VICAIR[®]
the best in body support

BUTTERFLY

GB

User manual

NL

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D

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N

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S

Bruksanvisning

DK

Brugermanual

SF

Käyttöohje

F

Mode d'emploi

CE

AVAILABLE STANDARD MODELS

Academy Butterfly

AVAILABLE STANDARD SIZES

CM	INCH
40x20	16x8
45x20	18x8
50x20	20x8

* Width x Height

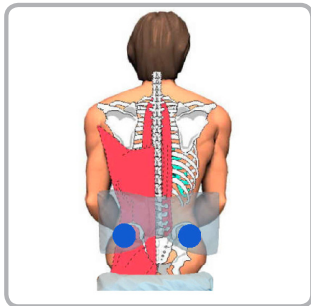
fig. A

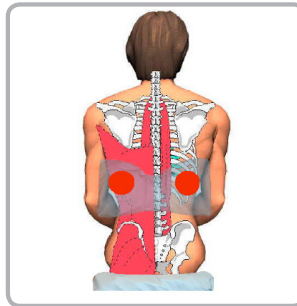
fig. B

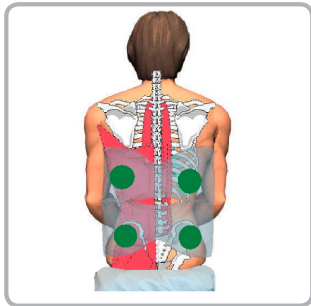
fig. C

INTRODUCTION

Your Vicair® Academy Butterfly is designed to be used as described in this manual. Please read the whole manual before use.

The Butterfly consists of the following parts:

Back cover & inner cushion

The back cover and the inner cushion are integrated in one part. The back cover is fitted with two straps to attach the Butterfly around the back rest of the (wheel)chair. The inner cushion is made up of zippered compartments. Each compartment is filled with a specific number of triangular, air filled SmartCells™.

Outer cover

The outer cover is made of Comfair, a breathable fabric which is two-way stretchable. The outer cover is zipped to the back cover.

INSTALLATION

Vicair recommends installation and adjustment of the Academy Butterfly under the direct supervision of your therapist or dealer.

- Remove the clear plastic wrapper.
- Make sure the Butterfly fits the dimensions of your body and your (wheel)chair.
- Place the Butterfly against the back rest of the (wheel)chair. Make sure the back label faces down.
- Close the buckles and tighten the back straps around the back rest of the (wheel)chair.

Seating positions

The Butterfly can be used for various seating positions:

Active seating position (fig. A)

To support the pelvis and the lower spine. Fasten the Butterfly low on the back rest.

Relaxed seating position (fig. B)

To support the lower part of the rib cage. Fasten the Butterfly upside down and slightly higher on the back rest.

Stable seating position (fig. C)

To support the pelvis, the lower spine and the ribcage. Fasten two Butterflies (one low, one high) on the back rest.

Checking for “bottoming out”

Your back and spine should not touch the back of the Butterfly. Checking for “bottoming out” should be done during first use and after adjustments to the cushion filling.

- Sit with your back against the Butterfly for approximately 20 minutes.
- Lean slightly forward in your (wheel)chair seat.
- Let your therapist or dealer slide a hand between the Butterfly and your back.
- Make sure your back is supported by at least one full layer of SmartCells□.
- Do not use the Butterfly if bottoming out occurs. In most cases, adjusting the filling grade of the compartments will solve the problem.

Adjusting the cushion filling

The Butterfly is overfilled with SmartCells™. If necessary, the firmness can be adjusted by removing or adding cells from the various compartments of the inner cushion.

USE

Checking for redness of the skin

Check your skin regularly for redness, especially during the first few days. If redness occurs and does not disappear after 15 minutes, stop using the Butterfly. Consult your therapist or dealer.

Use at high altitude

Because each SmartCell™ is filled with air, the Butterfly may behave differently when used at high altitude or in an aircraft. Under such circumstances, the cushion’s firmness can be adjusted by removing a few cells from each compartment. Vicair® recommends not to use the Butterfly above 2400 m (8000 ft) ASL (above sea level).

Use in motor vehicles

No testing data are currently available about the safe use of the Butterfly in motor vehicles. Therefore we do not encourage using the Butterfly in motor vehicles.

Transferring the Butterfly to another user

Before allowing a second person to use the Butterfly, clean the inner cushion and wash the outer cover. Disinfect both parts. Make sure the cushion is correctly adjusted to the second person.

Replacing the outer cover

Under normal use, Vicair recommends replacement of the outer cover every year. Replacement covers can be purchased separately.

Flammability

Do not use or leave the Butterfly close to a naked flame or a high temperature heat source.

MAINTENANCE AND CLEANING

Caution: Do not steam clean or autoclave the Butterfly. The Butterfly cannot withstand excessive heat or pressure.

Checking for wear and broken cells

Every three months, check the Butterfly for wear and collapsed cells. In case of serious damage, or a large number of collapsed cells, do not continue to use the Butterfly. Consult your dealer.

Cleaning the inner cushion

- Wipe the inner cushion with a damp cloth using a mild detergent. Difficult spots or stains can be cleaned by first saturating the spot and then using a soft brush to loosen the dirt.
- If it is necessary to remove the cells for cleaning, note the number of cells in each compartment.
- Do not use aggressive or abrasive cleaners, or sharp objects to clean the inner cushion.
- Do not immerse the inner cushion in water or any other liquid.
- Do not wash the filled inner cushion, or the cells, in a washing machine.

Cleaning the outer cover

The outer cover can be machine washed. See the instructions on the label attached to the inside of the cover.

Disinfecting the Butterfly

In case of soiling with body fluids, and when transferring the Butterfly to another user, the Butterfly must be disinfected to avoid cross contamination.

- Clean the inner cushion and the outer cover as described above. When washing the outer cover, preferably use an anti-bacterial agent.
- Spray disinfectant on the outer cover and the inner cushion. Bend the compartments outwards to make sure that the spray mist gets into the gaps.

WARRANTY

Vicair offers a limited warranty on defects in workmanship and materials for a period of two (2) years after the original date of purchase, provided the product has been used normally. Any damages arising from abnormal use, or caused by improper handling, cleaning or negligence are excluded from this warranty. Should a defect in materials or workmanship occur within two (2) years from the original date of purchase, Vicair will, at its own choice, repair or replace the product free of charge. The outer cover is excluded from this warranty.

Remedies for breach of express warranties herein are limited to repair or replacement of the product. In no event shall damages for breach of any warranty include any consequential damages or exceed the cost of non-conforming goods sold.

For additional product information, see our website: www.vicair.com.

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