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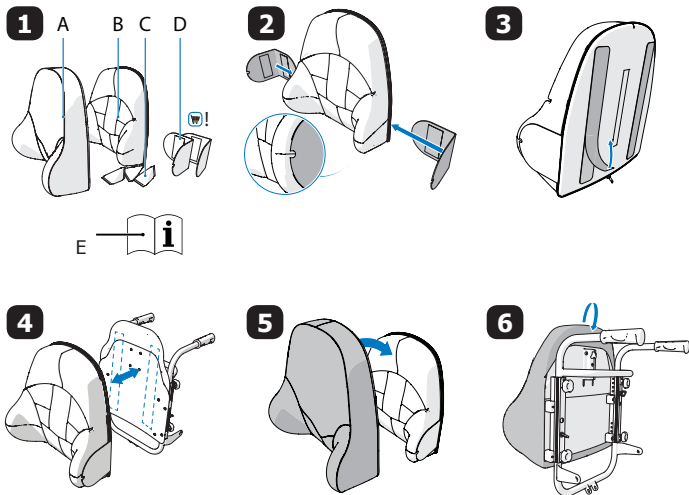
en User manual Vicair® Flex Back

Dear customer, thank you for buying a Vicair® Flex Back cushion. The Vicair® Flex Back is a back cushion intended for use in wheelchairs. It is developed for use in combination with an Invacare Flex back, but can also be used on other models of backrests. In this manual you will find all information needed to quickly become familiar with this product. Make sure that you read and understand this manual completely before use. Vicair recommends to assemble, install and adjust the Flex back cushion under the direct supervision of your therapist or dealer. Vicair BV can not be held responsible for damage caused by misuse, improper use or negligence. For more information, or to order documents, contact Vicair BV. Store the manual in a safe place for future reference.

Installation

The product consists of an inner cushion covered by a fabric outer cover. The inner cushion is filled with SmartCells™. During installation, SmartCells may be inserted or taken out of the inner cushion to ensure seating comfort. This may only be done by a certified professional.

OPTIONAL for users of side supports:

- Unzip the outer cover from the inner cushion.
 - Insert side supports in the gaps of the inner cushion. (see figure 2) Ensure the groove in the side supports aligns with the seam in the inner cushion. (see figure 2)
 - Use the included double sided tape to fix the side supports of the inner cushion in place.
1. Remove the strip covering the double sided tape on the backside of the inner cushion. (see figure 3) This double sided tape is attached to a Velcro strip.
 2. Align the cushion with the backrest. Then press them together so the double sided tape will stick to the backrest. (see figure 4)
 3. Carefully remove the cushion from the backrest. Ensure the Velcro remains stuck to the backrest. (see figure 4)
 4. Zip the outer cover over the inner cover (see figure 5)
 5. Place the cushion against the backrest and fold the envelope on the back of the cushion over the backrest (see figure 6).

Use

- ⚠ **Caution!** Do not use the inner cushion without the outer cover.
- ⚠ **Caution!** Do not use or leave your cushion close to a naked flame or high temperature heat source.
- ⚠ **Caution!** Do not use your cushion above 2.400 m (8.000 ft) ASL (above sea level).
- ⚠ **Caution!** The maximum user weight is 135 kg.
- ⚠ **Caution!** Do not use the cushion during transportation in motor vehicles.
- Check your skin regularly for redness, especially during the first days. If redness occurs and does not disappear after 15 minutes, do not continue to use the cushion. Consult your therapist or dealer.

- When transferring the cushion to another user disinfect the cushion.

Maintenance and cleaning

⚠ **Caution!** Do not steam or autoclave the cushion. The cushion can not withstand excessive heat or pressure.

⚠ **Caution!** Do not use aggressive or abrasive cleaners or sharp objects to clean the inner cushion.

- For light stains: clean the inner cushion with a damp cloth.
- For disinfection: disinfect the inner cushion with an anti bacterial detergent.
- The outer cover can be machine washed on 60 degrees. See the instructions on the label attached to the inside of the cover.
- Check your cushion every three months for wear and deflated cells. In case of serious damage or a large number of deflated cells, do not continue to use the cushion. Consult your dealer.
- Vicair recommends to replace the outer cover every year.

Warranty

Vicair offers a limited warranty against defects in workmanship and materials for a period of (2) two years after the original date of purchase. For general warranty conditions see our website: www.vicair.com.

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