

Olympus Folding Bariatric Bed Installation Guide

Remove all packaging from the bed and position the bed in the area ready for setting up – **Please note to leave room for the bed to fully unfold.**



Now press down on the castor brakes on the head end section placing them in the locked down position – **Please note to leave the castor brakes unlocked on the foot end section**



Unclip the red transport clip to unfold the bed – **Please note to fold the clip back against the frame**



Now using the hand set press and hold the back rest down button to unfold the bed. - **Please not to be carefull of surrounding area to ensure there is adequet space and no obsticals during unfolding.**



Now lock the foot end castors and remove the locking pin in the top centre of the back rest platform and the bed is now ready to be used. - **Integral bed rails can now be fitted if needed.**



Olympus Bariatric Bed Disassembly Guide

Lower the bed into it's lowest position and all profiling sections into the flat position – **Ensure bed rails have been removed if fitted**



Now ensure the castor brakes on the head section are locked in the down position – **Please note to leave the castor brakes are unlocked on the foot end section**



Now fit the locking pin in the top centre of the back rest platform



Now using the hand set press and hold the back rest up button to fold the bed. - **Please not to be carefull of surrounding area to ensure there is adequet space and no obsticals during folding.**



Now clip the red transport clip to to lock the bed in transport mode – **The bed can now be moved for tranportation**

