

Spare Parts Catalogue

Version: 2012/04/04b EN



Invacare® **Scanbed 755™**



Yes, you can.®

General Information

For any order of spare parts the following information is required:

- Product type*
- Product name & model*
- Serial Number*
- Part-Number*
- Part-Description*

Exploded view drawings and code numbers are not contractual and subject to modification, in relation to Invacare® continuous product improvement policy to meet end users needs.

The reproduction or use of this work, or of any substantial part of this work, in any form or by any electronic, mechanical or other means, including photocopying, scanning and recording, and in any information storage and retrieval system, including electronic transmission, is forbidden without the prior written permission of Invacare® International Sarl.



Yes, you can.®

TABLE OF CONTENTS

Colour specifications

Steel and plastic colours	5
Wood types	6

Mattress supports

Mattress support for 85+90 cm	7
Top frame, head end	8
Backrest	9
Lifting arm for backrest	10
Top frame, head end with CPR	11
Backrest	9
Motor for CPR backrest	12
Lifting arm for CPR backrest	13
Top frame, leg end	14
Mattress support, long, 105+120 cm	15
Top frame, head end	16
Lifting arm for backrest, width	17
Top frame, leg end	18

Bases

Base	19
Dividable base	20
Base, long version, 105 cm	21
Base, long version, 120 cm width	22

Castors and brake systems

Castors with brake	23
Ø80 mm castors with brake pedals	24
Ø100 mm castors with brake pedal	25
Ø125 mm castors with brake pedals	26
Central brake systems	27
Central brake system	28
Central brake systems with dir. control	29
Central brake system with direction control	30
Low slung kit	31
Ø75 mm low slung kit	32

Bed end plates

Sanne bed end	33
Susanne bed end	34
Sophie bed end	35
Emma bed end	36
Anita bed end	37
Victoria bed end	38

Electrical parts

Motor for back rest	39
Motor for back rest	40



Yes, you can.®

TABLE OF CONTENTS

Motor for thigh section	41
Motor for thigh section	42
Motor for leg section	43
Motor for leg section	44
Motor for height adjustment	45
Motor for height adjustment	46
Motor for height adjustment, 120 cm width	47
Motor for CPR backrest	12
Control unit	48
Control unit without accumulator backup, 4 motors, EU	49
Control unit without accumulator backup, 4 motors, UK	50
Control unit prepared for accumulator backup, 4 motors, EU	51
Control unit prepared for accumulator backup, 4 motors, UK	52
Hand control	53
Extension cable and Y-cable	54
ACP box	55
Battery backup	56

Side rails

Line siderail	57
Line siderail extendable	58
Britt V side rail	59
Scala steel side rail	60
Scala Basic 2, side rail	61
Scala Medium 2, side rail	62
Scala Decubi 2, side rail	63
Scala Basic Plus 2, side rail	64







Lifting pole

Lifting pole, Carebed, light	65
--	----

Accessories


Drip rod	66
Bumper wheel	67
Support handle	68
Cable holder	69

Steel and plastic colours

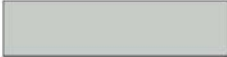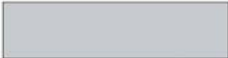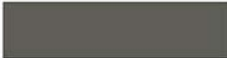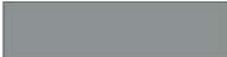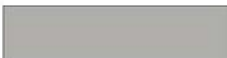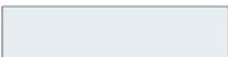Chromated, glossy 0001		0153		7035
Zinc, natural 0002		0154		7039
Zinc, black 0003		1015		7042
Anodized, black 0004		3004		7046
Anodized, natural 0005		5002		8016
Chromated, matte 0011		7012		9005
		7016		9006
		7032		9010
	0152	NB: Colours are for reference only Not all colours are available for all products		


Pos.	Part number	Description	Valid from → until	Qty
-		Not for separate sale - please contact Invacare		1

Wood types

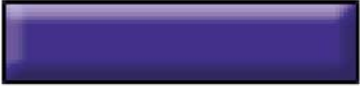	0101		0112
	0102		0113
	0103		0114
	0105		0115
	0110		0116
	0111		0117

Pos.	Part number	Description	Valid from → until	Qty
-		Not for separate sale - please contact Invacare		1


Mattress support for 85+90 cm

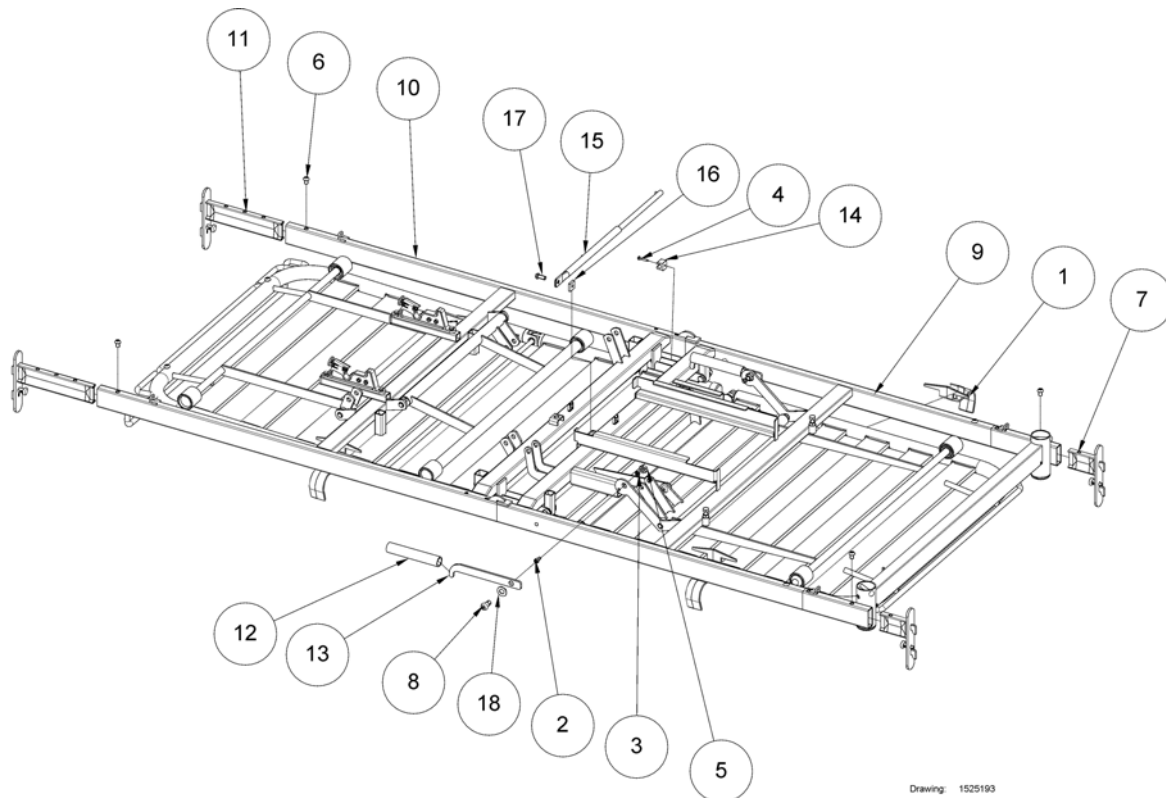

Drawing: 1525193


Pos.	Part number	Description	Valid from → until	Qty
9	-	Top frame, head end		1
9	-	Top frame, head end with CPR		0
10	-	Top frame, leg end		1
1	320583-7035	Mattress retainer for 85 cm width, RAL 7035		4
1	320584-7035	Mattress retainer for 90 cm width, RAL 7035		4
2	803032	Screw M6x12 St. 8.8 ISO 4762		2
3	803992	Nut M6 ISO 10511 self locking		2
4	806604	Screw Ø4,2x25 ISO 7051		1
5	806651	Lock nut M10		2
6	812875	Screw CBHc M8 x 12 mm 10.9		4
7	1443879-0002	Insertion tube with quick lock, head end		2
8	1493130	Pillar bolt Ø12		2
9	1446475	Washer Ø12xØ19x0,5		4
11	1516911-0002	Insertion tube with quick lock, bed end		2
12	1517076	Tube Ø18,5 x Ø22,5 x 155		2
13	1517256-0154	Hook		2
14	1538355	Part not for sale		1
15	1538445	Telescopic distance stick		1
16	1548667	Square nut, M8		1
17	1548674	Screw, M8x20 ISO7380		1
18	6750228	Nylon piece, 12,2x24x1,3		2

Top frame, head end

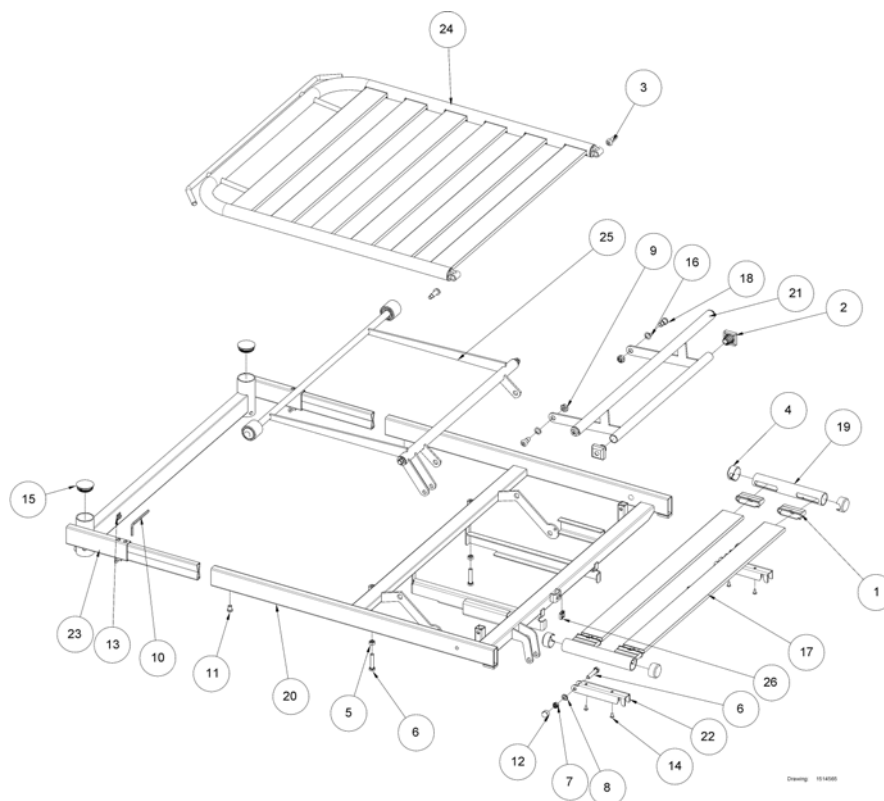


Pos.	Part number	Description	Valid from → until	Qty
24	-	Backrest		1
25	-	Lifting arm for backrest		1
1	314884-7035	Bracket for lamella, RAL 7035		4
2	319155-7035	Grey slide shoe		2
3	420623	screws/pillar/bolt for Invacare SB750		2
4	421460-7035	End plug Ø35,5/Ø32,5 RAL 7035		4
5	802729	Nut M8 for SB750		2
6	803008	Set screw M8x40 St. 8.8 ISO 4017		4
7	803927	Locknut M8 Z HFC 840 DIN 985-8		2
8	804200	Washer ISO7090 M08 zn		2
9	806651	Lock nut M10		2
10	811408	Hexagonal key 5 mm		1
11	812875	Screw CBHc M8 x 12 mm 10.9		2
12	814395-7035	Nut cap M8, RAL 7035		2
13	816488	Cable clips 4,6x4,1		1
14	1423996	Blind rivet Ø4,8x9		4
15	1432700-7035	Ferrule Ø44/Ø40x21,2 RAL 7035		2
16	1446475	Washer Ø12xØ19x0,5		2
17	1484059	Lamella for mattress support, 705 mm, aluminium		2
17	420628	Lamella for mattress support, 705 mm, wood		2
18	1493130	Pillar bolt Ø12		2
19	1514827	Tube for seat section		2
20	1518118-0154	Top frame head end, 85 cm		1
20	1515103-0154	Top frame head end, 90 cm		1
21	1515130-0154	Steering arm, head end		1
22	1515141-0002	Console for seat section, SB755		2
23	1518120-0154	Bed end bracket, head end, 85 cm		1
23	1515233-0154	Bed end bracket, head end, 90 cm		1
26	1538423	Clips for cables		1

Backrest

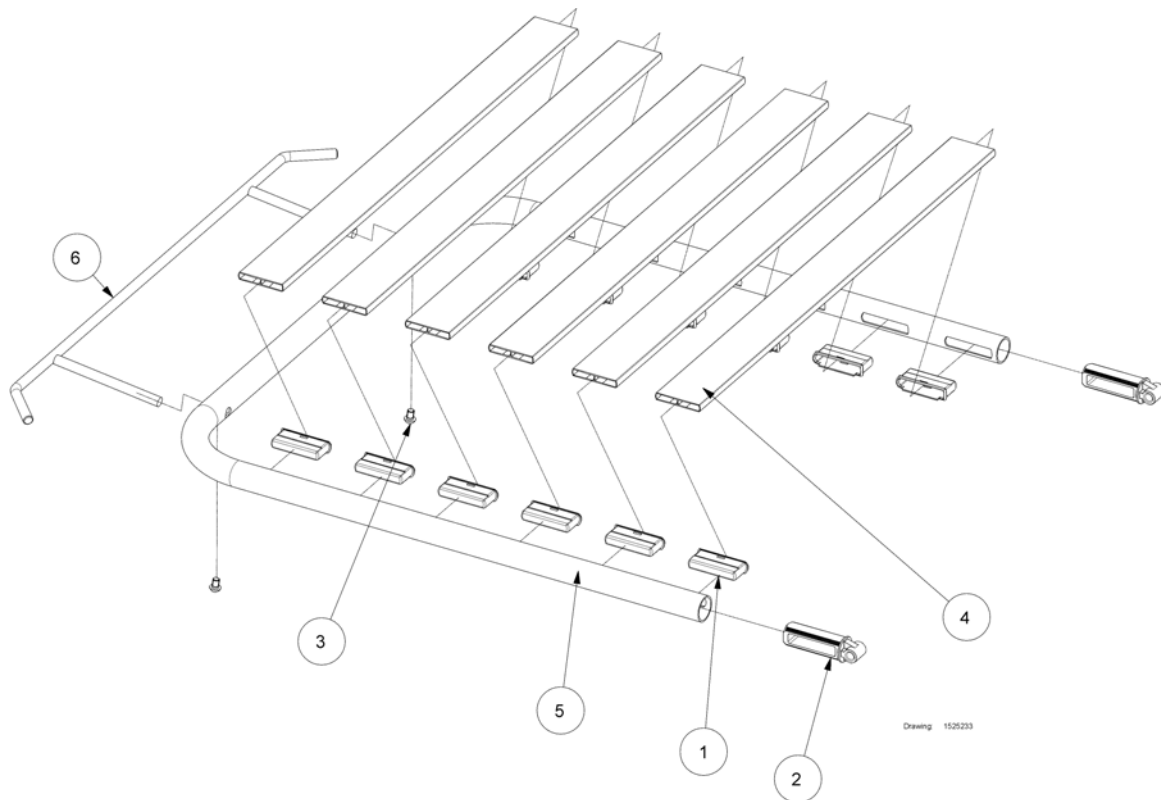


Pos.	Part number	Description	Valid from → until	Qty
1	314884-7035	Bracket for lamella, RAL 7035		12
2	319800	Hinged joint for mattress support		2
3	816420	Screw M8x14 st.10.9		2
4	1484059	Lamella for mattress support, 705 mm, aluminium		6
4	420628	Lamella for mattress support, 705 mm, wood		6
5	1516895	Tubular bow for backrest		1
6	1516913-0005	Mattress brace, head end, SB755		1

Lifting arm for backrest

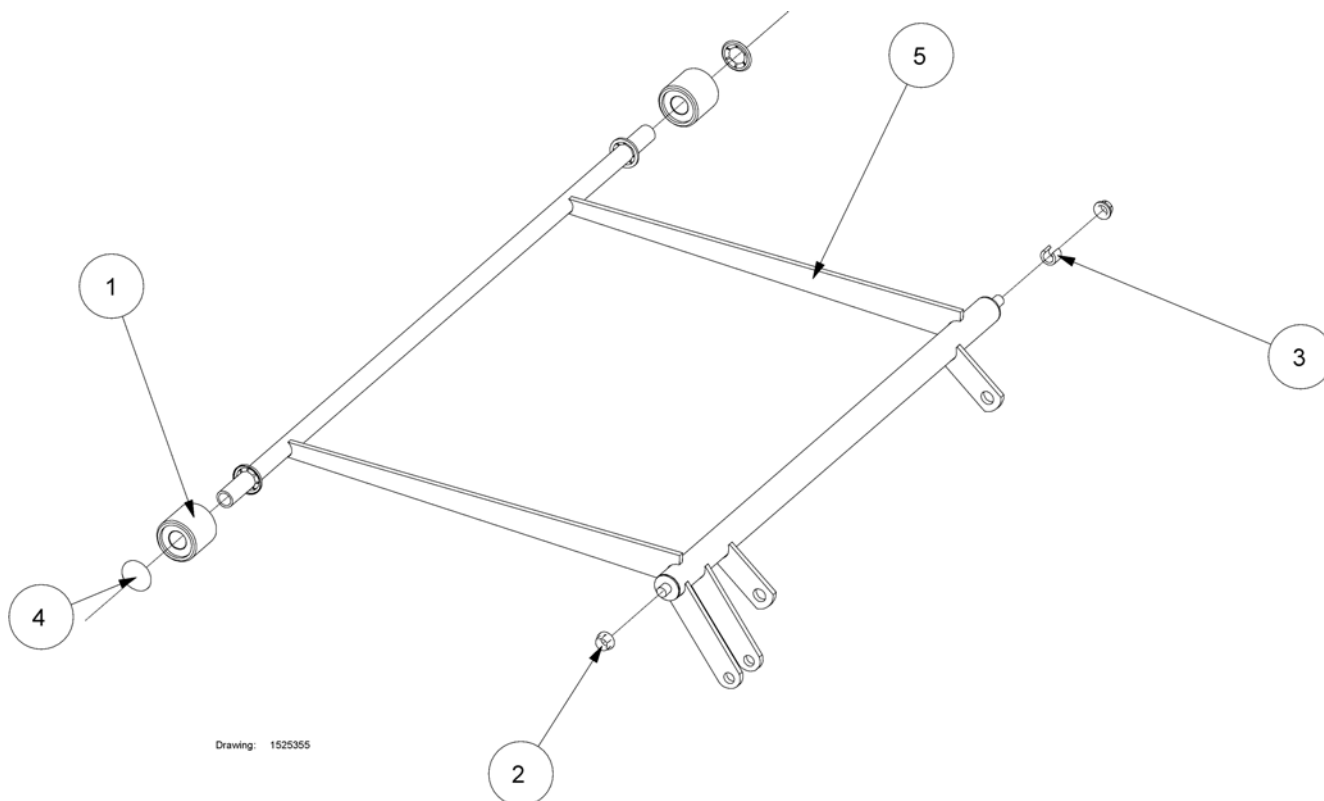


Pos.	Part number	Description	Valid from → until	Qty
1	405479-7035	Plastic roll Ø40/Ø17x38 RAL 7035		2
2	420625-7035	Bearing bushing Ø10,2/Ø12,9/Ø16,5x5,7 RAL 7035		2
3	420649-7035	Fixation tube Ø15/Ø10x10 slotted, RAL 7035		1
4	815764	Starlock button Ø16		2
5	1515123-0154	Lifting arm for backrest		1

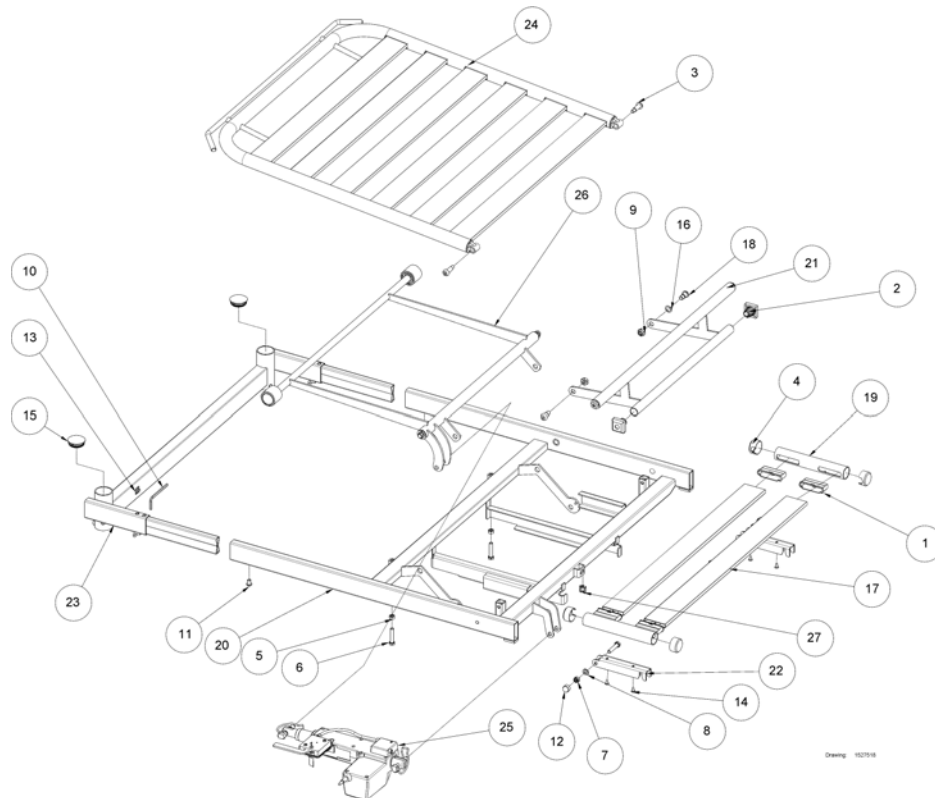Top frame, head end with CPR


Pos.	Part number	Description	Valid from → until	Qty
24	-	Backrest		0
25	-	Motor for CPR backrest		1
26	-	Lifting arm for CPR backrest		0
1	314884-7035	Bracket for lamella, RAL 7035		1
2	319155-7035	Grey slide shoe		1
3	420623	screws/pillar/bolt for Invacare SB750		1
4	421460-7035	End plug Ø35,5/Ø32,5 RAL 7035		1
5	802729	Nut M8 for SB750		1
6	803008	Set screw M8x40 St. 8.8 ISO 4017		1
7	803927	Locknut M8 Z HFC 840 DIN 985-8		1
8	804200	Washer ISO7090 M08 zn		1
9	806651	Lock nut M10		1
10	811408	Hexagonal key 5 mm		1
11	816420	Screw M8x14 st.10.9		1
12	814395-7035	Nut cap M8, RAL 7035		1
13	816488	Cable clips 4,6x4,1		1
14	1423996	Blind rivet Ø4,8x9		1
15	1432700-7035	Ferrule Ø44/Ø40x21,2 RAL 7035		1
16	1446475	Washer Ø12xØ19x0,5		1
17	1484059	Lamella for mattress support, 705 mm, aluminium		1
17	420628	Lamella for mattress support, 705 mm, wood		1
18	1493130	Pillar bolt Ø12		1
19	1514827	Tube for seat section		1
20	1518118-0154	Top frame head end, 85 cm		1
20	1515103-0154	Top frame head end, 90 cm		1
21	1515130-0154	Steering arm, head end		1
22	1515141-0002	Console for seat section, SB755		1
23	1518120-0154	Bed end bracket, head end, 85 cm		1
23	1515233-0154	Bed end bracket, head end, 90 cm		1
27	1538423	Clips for cables		1



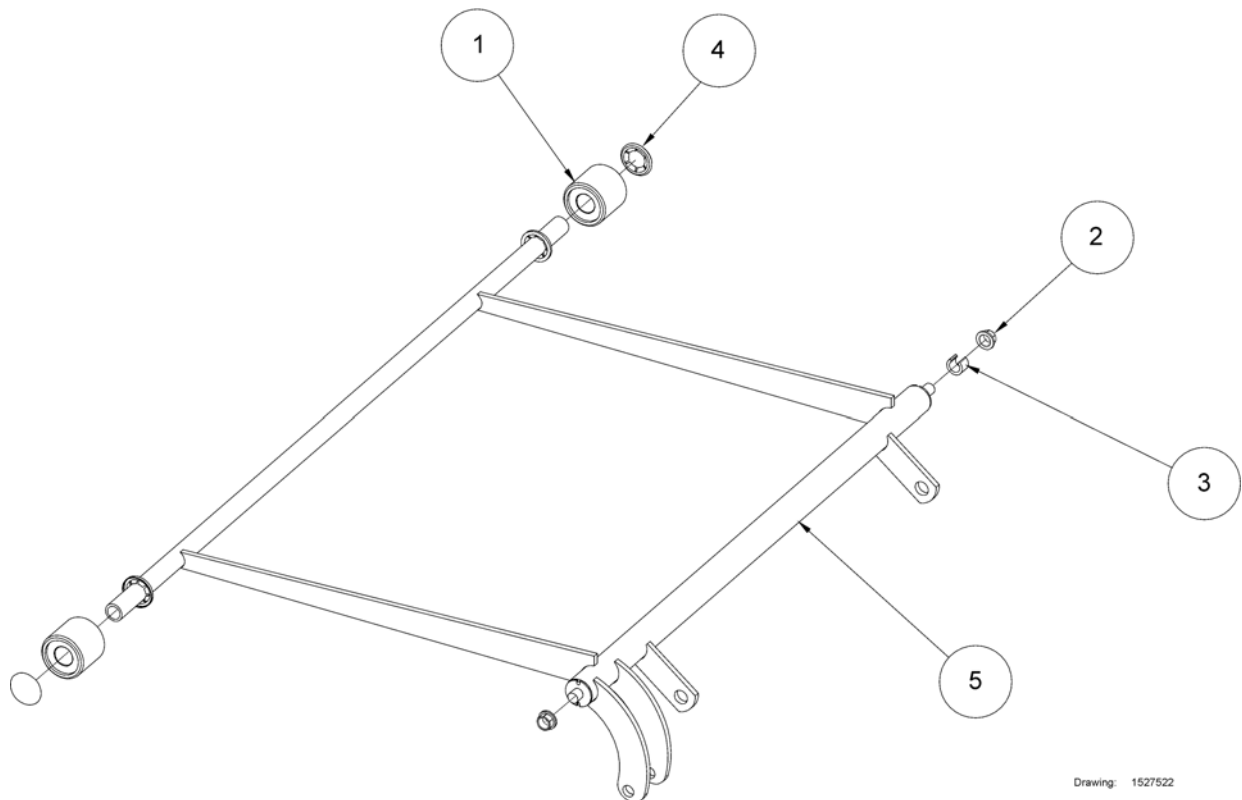

Pos.	Part number	Description	Valid from → until	Qty
1	405479-7035	Plastic roll Ø40/Ø17x38 RAL 7035		1
2	420625-7035	Bearing bushing Ø10,2/Ø12,9/Ø16,5x5,7 RAL 7035		1
3	420649-7035	Fixation tube Ø15/Ø10x10 slotted, RAL 7035		1
4	815764	Starlock button Ø16		1
5	1527523	Lifting arm		1

Top frame, leg end

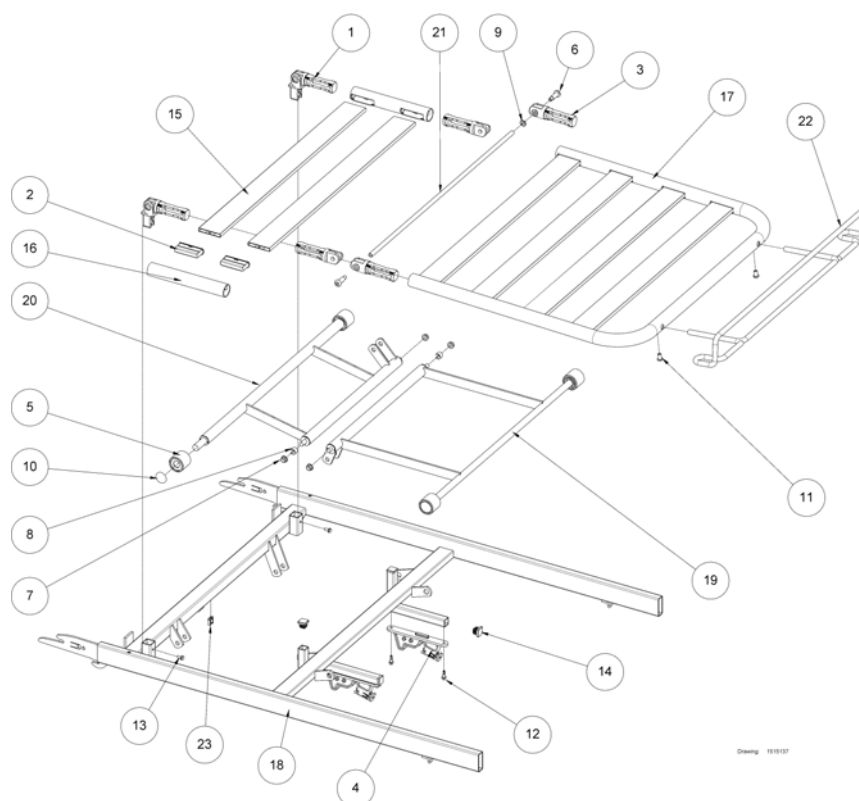


Drawing 1510107

Pos.	Part number	Description	Valid from → until	Qty
1	021351	Hinged joint for mattress support		2
2	314884-7035	Bracket for lamella, RAL 7035		12
3	314885	Hinged joint for mattress support		4
4	320529-7035	Locking bracket for mattress support/base, RAL 7035		2
5	405479-7035	Plastic roll Ø40/Ø17x38 RAL 7035		4
6	420623	screws/pillar/bolt for Invacare SB750		2
7	420625-7035	Bearing bushing Ø10,2/Ø12,9/Ø16,5x5,7 RAL 7035		4
8	420649-7035	Fixation tube Ø15/Ø10x10 slotted, RAL 7035		2
9	804200	Washer ISO7090 M08 zn		2
10	815764	Starlock button Ø16		4
11	816420	Screw M8x14 st.10.9		2
12	819407	Screw Ø5,5x19 ISO 15480		4
13	821235	Screw Ø5x12, thread forming		2
14	1423915	Ferrule 20x20 RAL 7035		4
15	1484059	Lamella for mattress support, 705 mm, aluminium		6
15	420628	Lamella for mattress support, 705 mm, wood		6
16	1514847	Tube for thigh section, SB755		2
17	1514848	Tubular bow for leg section, SB755		1
18	1518124-0154	Top frame, leg section, 85 cm, SB755		1
18	1515132-0154	Top frame, leg section, 90 cm, SB755		1
19	1515134-0154	Lifting arm, leg section, SB755		1
20	1515136-0154	Steering arm, leg section, SB755		1
21	1515184	Tube Ø10/Ø7x687, SB755		1
22	1516844-0005	Mattress retainer, foot end, SB755		1
23	1538423	Clips for cables		1

Mattress support, long, 105+120 cm

Pos.	Part number	Description	Valid from → until	Qty
-		Top frame, head end		1
-		Top frame, leg end		1


Top frame, head end

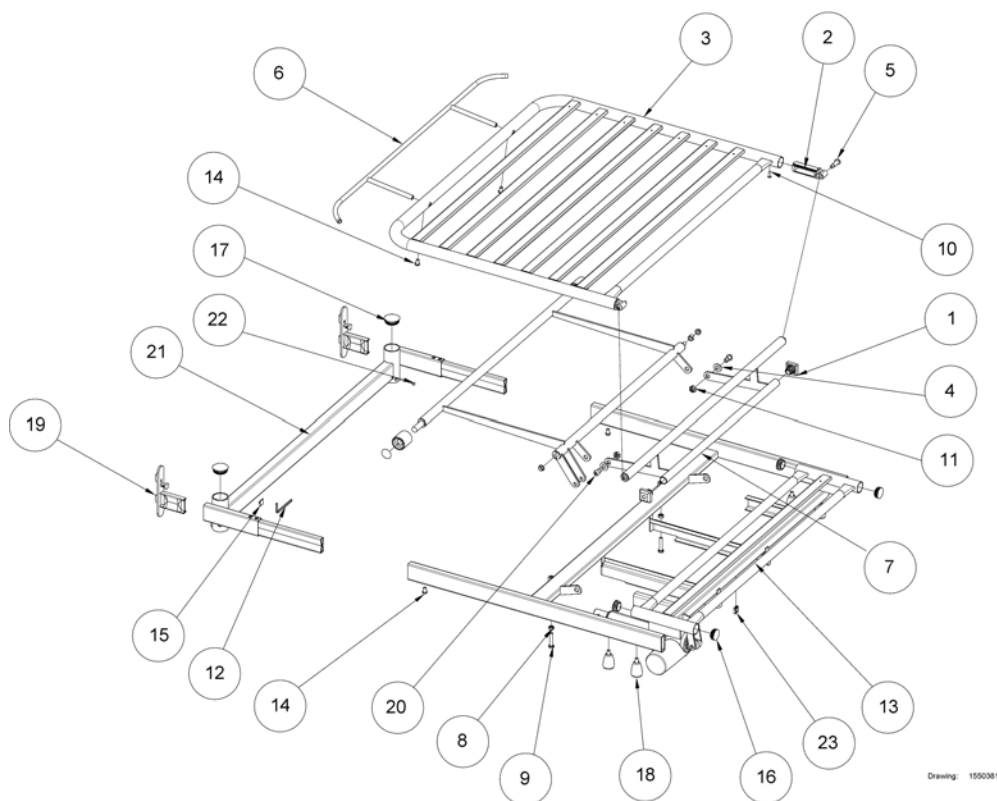

Pos.	Part number	Description	Valid from → until	Qty
1	319155-7035	Grey slide shoe		2
-		Lifting arm for backrest, width		1
2	319800	Hinged joint for mattress support		1
3	SP320877-0154	Backrest, 105cm		1
3	SP320892-0154	Backrest, 120cm		1
4	6750228	Nylon piece, 12,2x24x1,3		2
5	420623	screws/pillar/bolt for Invacare SB750		2
6	1551830-0154	Part not for sale		1
7	SP1550894-0154	Steering arm, head end, 105cm		1
7	SP1551294-0154	Steering arm, head end, 120cm		1
8	802729	Nut M8 for SB750		2
9	803008	Set screw M8x40 St. 8.8 ISO 4017		2
10	803811	Rivet Ø4,8x16		2
11	806651	Lock nut M10		2
12	811408	Hexagonal key 5 mm		1
13	SP1550379-0154	Top frame, head end, 105cm		1
13	SP1550685-0154	Top frame, head end, 120cm		1
14	816420	Screw M8x14 st.10.9		4
15	816488	Cable clips 4,6x4,1		1
16	1423911	Grey cap Ø32 mm		4
17	1432700-7035	Ferrule Ø44/Ø40x21,2 RAL 7035		2
18	1435325-7035	Finger screw M8 x 13 mm RAL7035		4
19	1443879-0002	Insertion tube with quick lock, head end		2
20	1493130	Pillar bolt Ø12		2
21	SP1550378-0154	Bed end bracket, head end, 105cm		1
21	SP1550834-0154	Bed end bracket, head end, 120cm		1
22	1538319	Part not for sale		2
23	1538423	Clips for cables		1


Invacare® ScanBed 755 | Mattress supports

Lifting arm for backrest, width

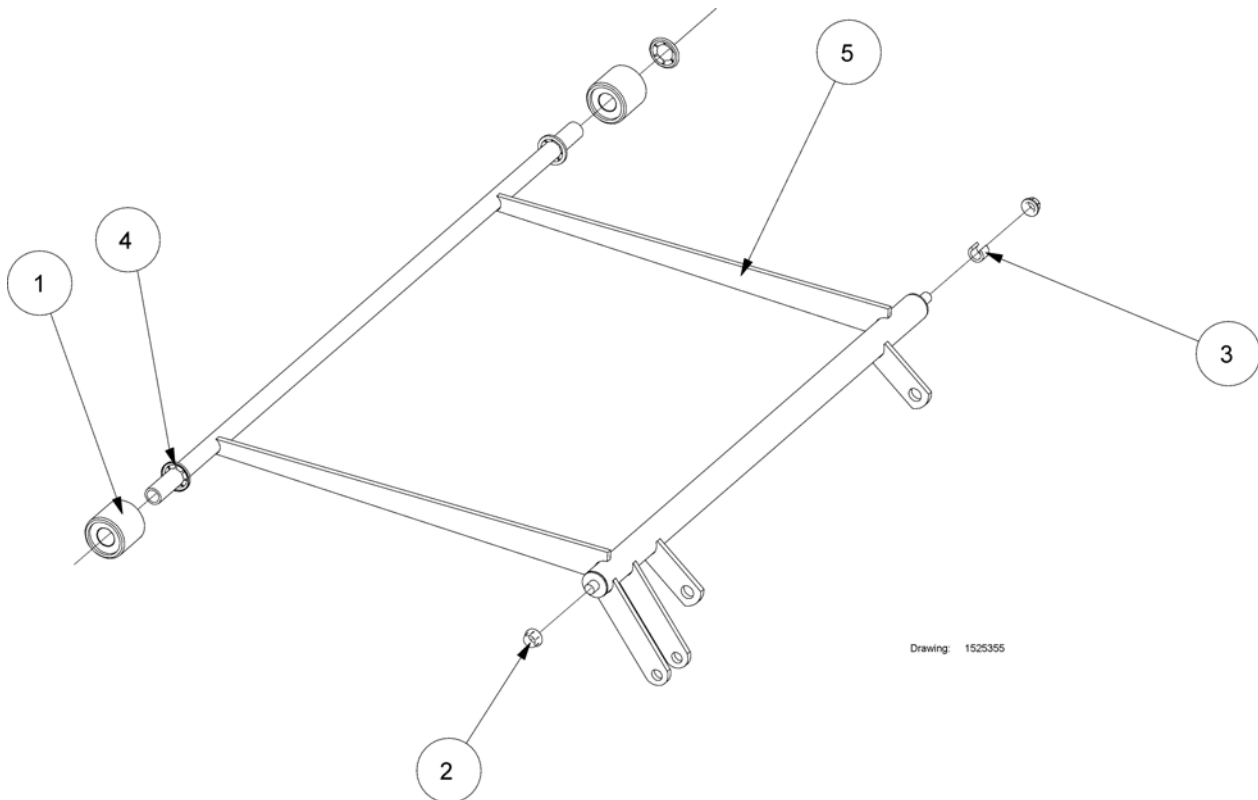


Pos.	Part number	Description	Valid from → until	Qty
1	405479-7035	Plastic roll Ø40/Ø17x38 RAL 7035		2
2	420625-7035	Bearing bushing Ø10,2/Ø12,9/Ø16,5x5,7 RAL 7035		2
3	420649-7035	Fixation tube Ø15/Ø10x10 slotted, RAL 7035		1
4	815764	Starlock button Ø16		2
5	SP1550893-0154	Lifter arm, head end, 105cm		1
5	SP1551291-0154	Lifter arm, head end, 120cm		1

Top frame, leg end

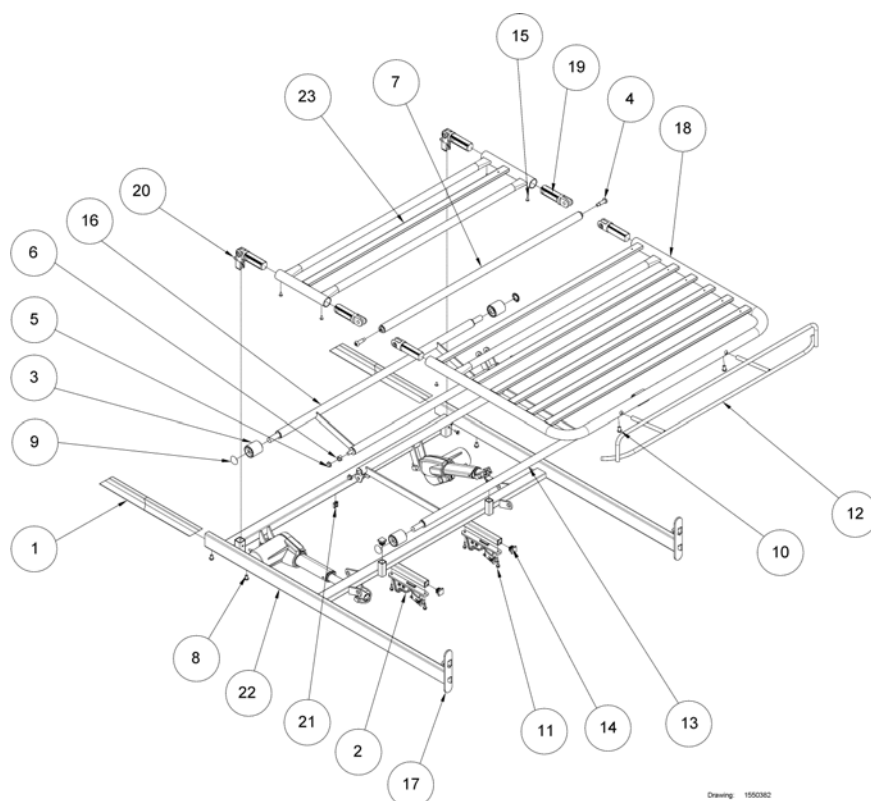


Pos.	Part number	Description	Valid from → until	Qty
1	320516.60	Insert 45x15x1,5x400		2
2	320529-7035	Locking bracket for mattress support/base, RAL 7035		2
3	405479-7035	Plastic roll Ø40/Ø17x38 RAL 7035		4
4	420623	screws/pillar/bolt for Invacare SB750		2
5	420625-7035	Bearing bushing Ø10,2/Ø12,9/Ø16,5x5,7 RAL 7035		4
6	420649-7035	Fixation tube Ø15/Ø10x10 slotted, RAL 7035		2
7	SP420975-0154	Distance tube, long, 105cm		1
7	SP421430-0154	Distance tube, long, 120cm		1
8	812875	Screw CBHc M8 x 12 mm 10.9		4
9	815764	Starlock button Ø16		4
10	816420	Screw M8x14 st.10.9		4
11	819407	Screw Ø5,5x19 ISO 15480		4
12	1551827-0154	Part not for sale		1
12	1551829-0154	Part not for sale		1
13	SP1550896-0154	Lifter arm, leg end, 105cm		1
13	SP1551292-0154	Lifter arm, leg end, 120cm		1
14	1423915	Ferrule 20x20 RAL 7035		4
15	1423996	Blind rivet Ø4,8x9		6
16	SP1550895-0154	Lifter arm, thigh section, 105cm		1
16	SP1551293-0154	Lifter ar, thigh, 120 cm		1
17	1516911-0002	Insertion tube with quick lock, bed end		2
18	SP1550648-0154	Leg section, 105cm		1
18	SP1550683-0154	Leg section, 120cm		1
19	1529664	Hinged joint for mattress support		4
20	1530100	Hinged joint for mattress support		2
21	1538423	Clips for cables		1
22	SP1550380-0154	Top frame, leg section, 105cm		1
22	SP1550835-0154	Top frame, leg section, 120cm		1
23	SP1550645-0154	Thigh section, 105cm		1
23	SP1550682-0154	Thigh section, 120cm		1

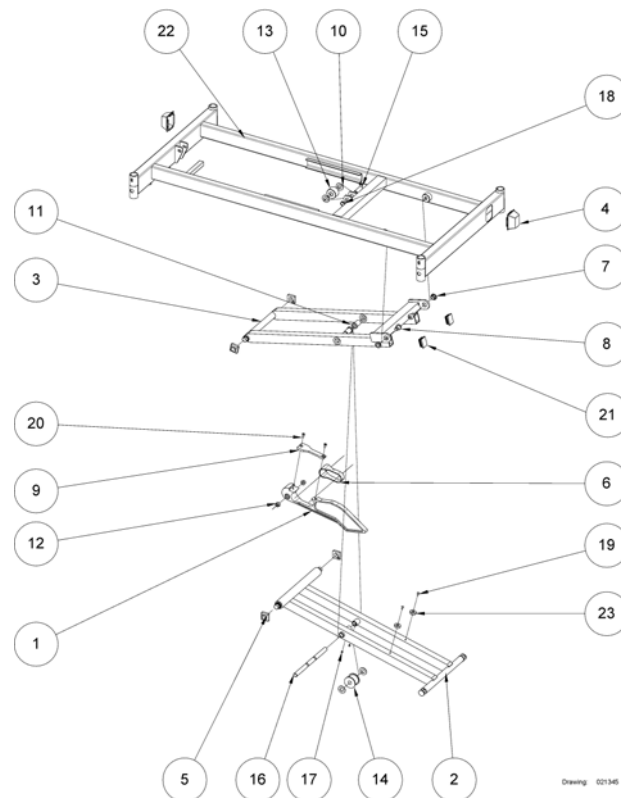
Base

Pos.	Part number	Description	Valid from → until	Qty
1	221103	Wedge, aluminium		1
2	221122-0154	Inner shear arm		1
3	221123-0154	Outer shear arm		1
4	315626-7035	Cover for central brake RAL7035		2
5	318648	Slide shoe 33,4x33,4x13,5		4
6	318673	Slide bearing for wedge		1
7	409645	Bushing Ø16,4xØ20xØ25		2
8	418477	Pillar bolt M12x22,3		2
9	418896	Wedge retainer, aluminium		1
10	419298	Washer Ø32/Ø16,5x2		4
11	420154	Bushing Ø25/Ø19,8xØ16,4		2
12	420171	Bearing bushing Ø10/Ø16/Ø19x8		2
13	421442	Plastic roll Ø42/Ø16,1x27		1
14	421443	Plastic roll Ø50/Ø42/Ø16,1x36		1
15	421452	Pin and angle for base		1
16	421455	Axle Ø16x250		1
17	803221	Screw M6x6 St. 14.9 ISO 4029		2
18	803247	Circlip ring Ø16x1		1
19	803833	Rivet Ø4,8x12		2
20	816401	Screw M6x16 St. 8.8		2
21	1423916	Ferrule 40x30x3 RAL 7035		2
22	1451305-0154	Non-dividable base		1
23	1507221	Distance piece		2

Dividable base

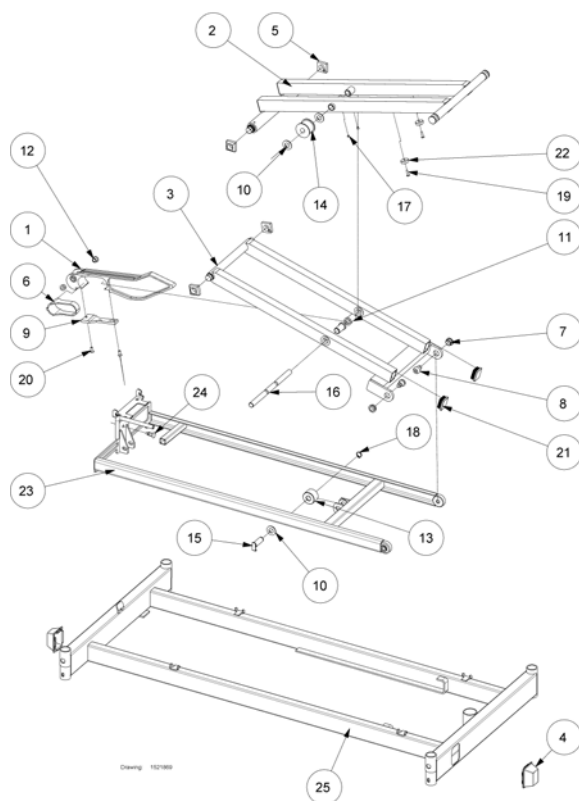


Pos.	Part number	Description	Valid from → until	Qty
1	221103	Wedge, aluminium		1
2	221122-0154	Inner shear arm		1
3	221123-0154	Outer shear arm		1
4	315626-7035	Cover for central brake RAL7035		2
5	318648	Slide shoe 33,4x33,4x13,5		4
6	318673	Slide bearing for wedge		1
7	409645	Bushing Ø16,4xØ20xØ25		2
8	418477	Pillar bolt M12x22,3		2
9	418896	Wedge retainer, aluminium		1
10	419298	Washer Ø32/Ø16,5x2		4
11	420154	Bushing Ø25/Ø19,8xØ16,4		2
12	420171	Bearing bushing Ø10/Ø16/Ø19x8		2
13	421442	Plastic roll Ø42/Ø16,1x27		1
14	421443	Plastic roll Ø50/Ø42/Ø16,1x36		1
15	421452	Pin and angle for base		1
16	421455	Axle Ø16x250		1
17	803221	Screw M6x6 St. 14.9 ISO 4029		2
18	803247	Circlip ring Ø16x1		1
19	803833	Rivet Ø4,8x12		2
20	816401	Screw M6x16 St. 8.8		2
21	1423916	Ferrule 40x30x3 RAL 7035		2
22	1507221	Distance piece		2
23	1513526-0154	Intermediate frame for SB755		1
24	1513591	Quick lock, not lockable		1
25	1521868-0154	Dividable base, SB755		1

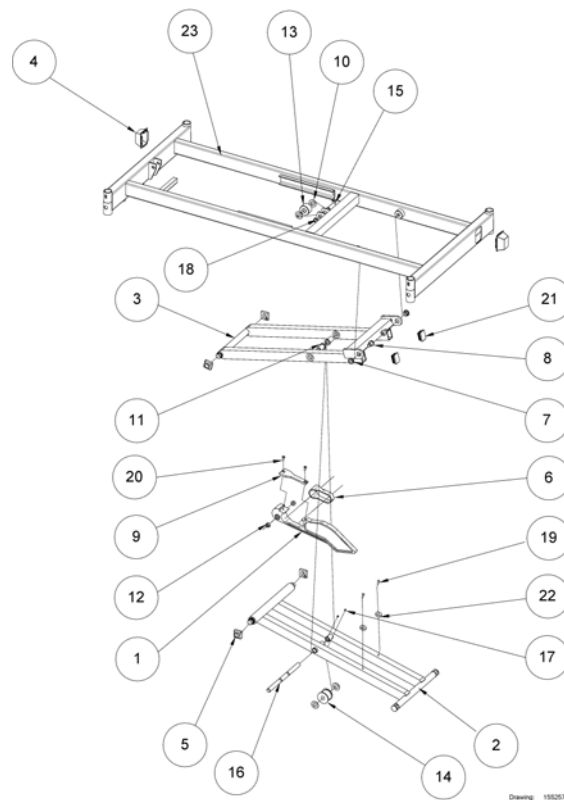Base, long version, 105 cm

Drawing: 1552578

Pos.	Part number	Description	Valid from → until	Qty
1	221103	Wedge, aluminium		1
2	221122-0154	Inner shear arm		1
3	221123-0154	Outer shear arm		1
4	315626-7035	Cover for central brake RAL7035		2
5	318648	Slide shoe 33,4x33,4x13,5		4
6	318673	Slide bearing for wedge		1
7	409645	Bushing Ø16,4xØ20xØ25		2
8	418477	Pillar bolt M12x22,3		2
9	418896	Wedge retainer, aluminium		1
10	419298	Washer Ø32/Ø16,5x2		4
11	420154	Bushing Ø25/Ø19,8xØ16,4		2
12	420171	Bearing bushing Ø10/Ø16/Ø19x8		2
13	421442	Plastic roll Ø42/Ø16,1x27		1
14	421443	Plastic roll Ø50/Ø42/Ø16,1x36		1
15	421452	Pin and angle for base		1
16	421455	Axle Ø16x250		1
17	803221	Screw M6x6 St. 14.9 ISO 4029		2
18	803247	Circlip ring Ø16x1		1
19	803835	Blind rivet 4,8x18		2
20	816401	Screw M6x16 St. 8.8		2
21	1423916	Ferrule 40x30x3 RAL 7035		2
22	1507221	Distance piece		2
23	SP1551940-0154	Base, long, 105cm		1

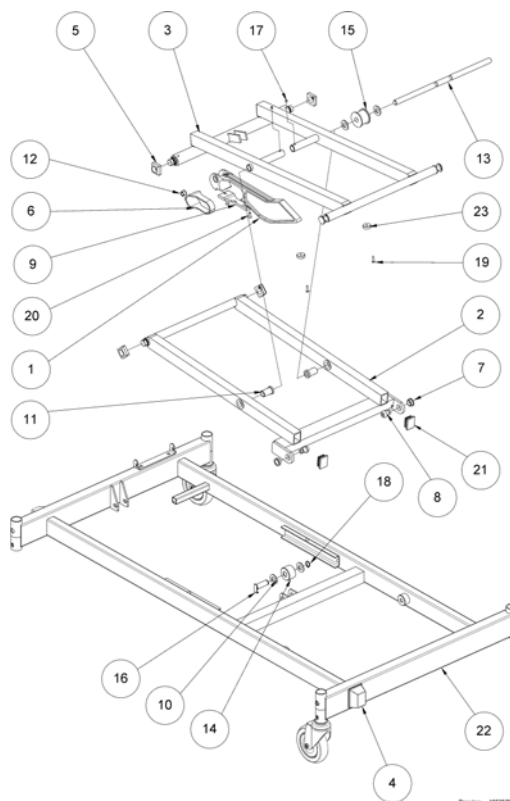Base, long version, 120 cm width


Drawing: 180209

Pos.	Part number	Description	Valid from → until	Qty
1	221103	Wedge, aluminium		1
2	SP221135-0154	Shear arm, outer, 120cm		1
3	SP221138-0154	Shear arm, inner, 120cm		1
4	315626-7035	Cover for central brake RAL7035		2
5	318648	Slide shoe 33,4x33,4x13,5		4
6	318673	Slide bearing for wedge		1
7	409645	Bushing Ø16,4xØ20xØ25		2
8	418477	Pillar bolt M12x22,3		2
9	418896	Wedge retainer, aluminium		1
10	419298	Washer Ø32/Ø16,5x2		4
11	420154	Bushing Ø25/Ø19,8xØ16,4		2
12	420171	Bearing bushing Ø10/Ø16/Ø19x8		2
13	421422	Axle Ø16x400, 120 cm width		1
14	421442	Plastic roll Ø42/Ø16,1x27		1
15	421443	Plastic roll Ø50/Ø42/Ø16,1x36		1
16	421452	Pin and angle for base		1
17	803221	Screw M6x6 St. 14.9 ISO 4029		2
18	803247	Circlip ring Ø16x1		1
19	803835	Blind rivet 4,8x18		2
20	816401	Screw M6x16 St. 8.8		2
21	1423916	Ferrule 40x30x3 RAL 7035		2
22	SP1551941-0154	Base, long, 120cm		1
23	1507221	Distance piece		2


Castors with brake

Pos.	Part number	Description	Valid from → until	Qty
-		Ø80 mm castors with brake pedals		0
-		Ø100 mm castors with brake pedal		0
-		Ø125 mm castors with brake pedals		0

Ø80 mm castors with brake pedals

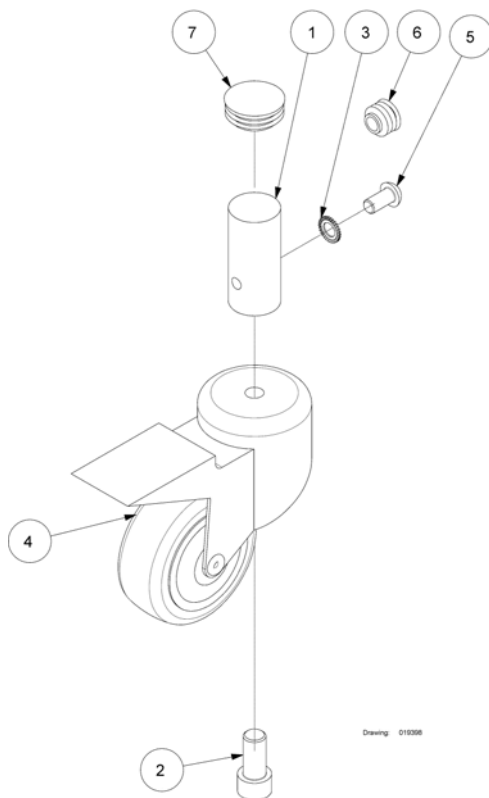


Pos.	Part number	Description	Valid from → until	Qty
1	418894-0154P	Part not for sale		1
2	803065	Screw M12x25 8.8 ISO 4762		1
3	803276	Locking washer M8		1
4	803931	Wheel Ø80 Rhombus with brake pedal		1
5	806645	Screw M8x16 ISO 7380		1
6	1423910-7035	Ferrule Ø18, RAL 7035		1
7	1423911	Grey cap Ø32 mm		1

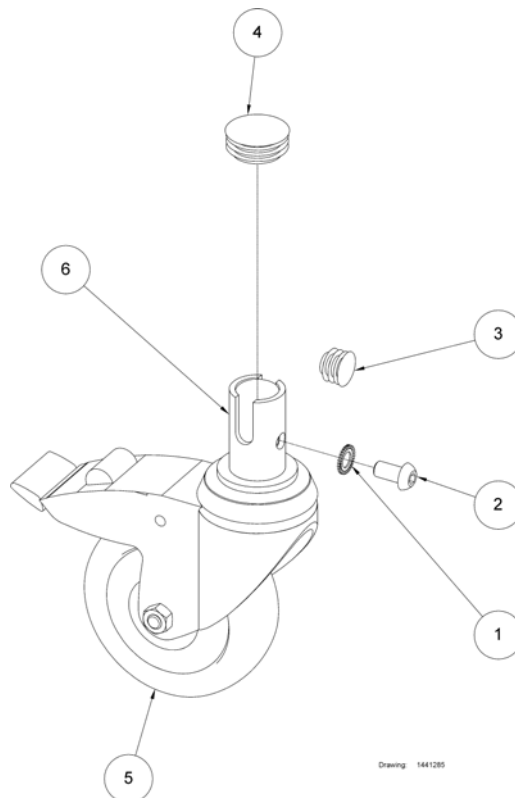Ø100 mm castors with brake pedal

Pos.	Part number	Description	Valid from → until	Qty
1	803276	Locking washer M8		1
2	806645	Screw M8x16 ISO 7380		1
3	1423910-7035	Ferrule Ø18, RAL 7035		1
4	1423911	Grey cap Ø32 mm		1
5	1441284	Castor Ø100 Tente, with brake pedal		1
6	1538398	Bushing, ø28/ø22x45		1

Ø125 mm castors with brake pedals

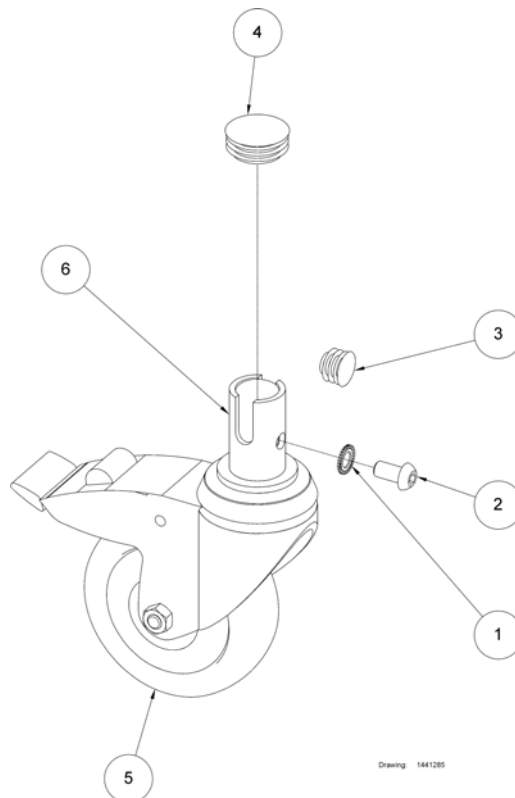


Pos.	Part number	Description	Valid from → until	Qty
1	803276	Locking washer M8		1
2	806645	Screw M8x16 ISO 7380		1
3	1423910-7035	Ferrule Ø18, RAL 7035		1
4	1423911	Grey cap Ø32 mm		1
5	1441346	Castor Ø125 Tente, with brake pedal		1
6	1538398	Bushing, ø28/ø22x45		1

Central brake systems

Pos.	Part number	Description	Valid from → until	Qty
-		Central brake system		1

Central brake system

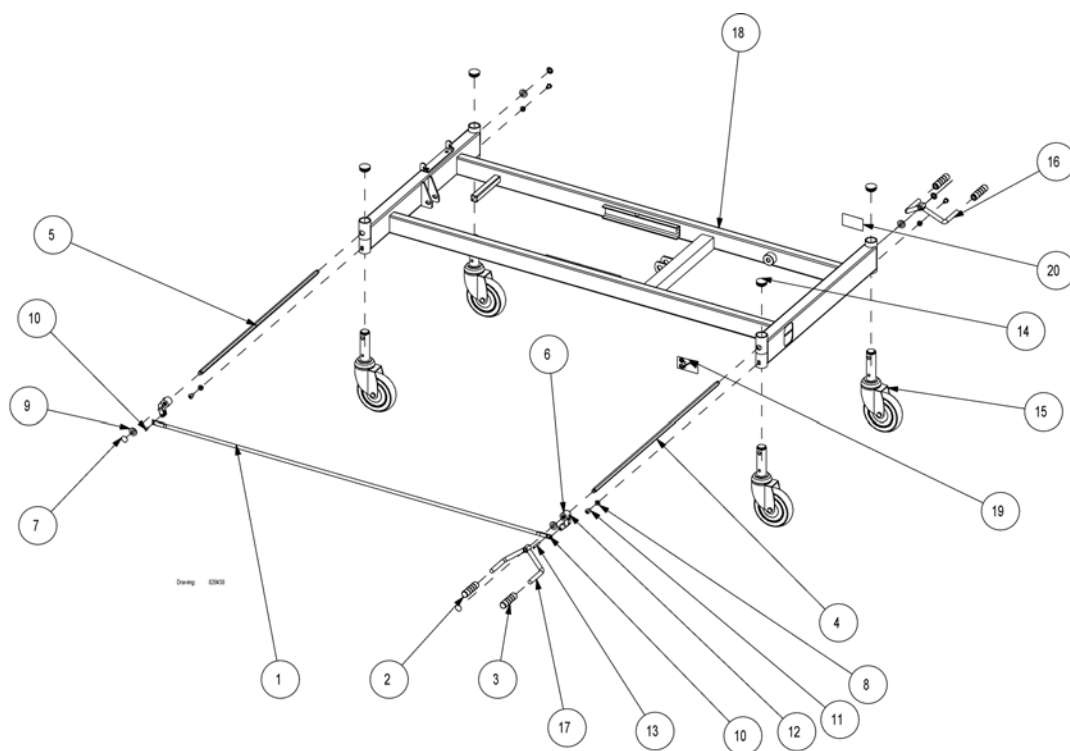


Drawing: 03003

Pos.	Part number	Description	Valid from → until	Qty
1	314284-1015	Central brake rod Ø10x1,5x1369 mm		1
1	SP1551970-0154	Central brake rod, Ø10x1,5x1449 mm		1
2	414411	Red cap for brake pedal		2
3	414412	Green cap for brake pedal		2
4	414652.60	Axle for central brake 740 mm		1
4	421488.60	Axle NV 11x850, 120 cm width		1
5	414653.60	Axle for central brake 700 mm		1
5	421489.60	Axle NV 11x890, 120 cm width		1
6	414654	Retainer for central brake		2
7	803279	Starlock button Ø10 mm		4
8	803876	Spring washer M8		4
9	804239	Nylon washer Ø13 x Ø24 x 2.5 mm		4
10	806644	Spiral clamp pin Ø5 x 14 mm		2
11	812875	Screw CBHc M8 x 12 mm 10.9		4
12	817243	Screw M6 x 8 mm		2
13	821213	Spiral clamp pin Ø5 x36 mm		1
14	1423911	Grey cap Ø32 mm		4
15	819460	Castor Ø100 mm for central brake		4
15	1423937	Castor Ø125 mm with central brake		4
15	1423988	Castor Ø150 Tente, with central brake		4
16	1524937-0154	Not for sepaeate sale, please contact Invacare		1
17	1524938-0154	Not for sepaeate sale, please contact Invacare		1
18	1451305-0154	Non-dividable base		1
18	1551940-0154	Base - Long version		1
19	1492781	Label for brake, 40x70, left		1
20	1492782	Label for brake, 40x70, right		1

Central brake systems with dir. control

Pos.	Part number	Description	Valid from → until	Qty
-		Central brake system with direction control		1

Central brake system with direction control

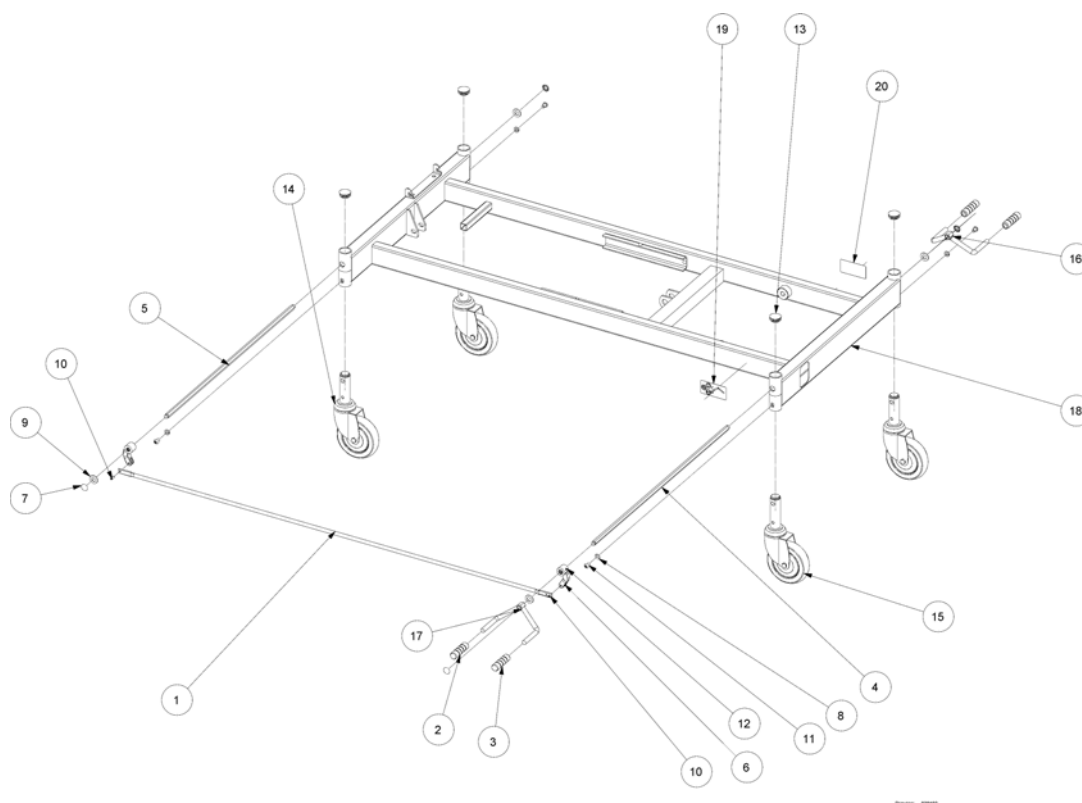


Pos.	Part number	Description	Valid from → until	Qty
1	314284-1015	Central brake rod Ø10x1,5x1369 mm		1
2	414411	Red cap for brake pedal		2
3	414412	Green cap for brake pedal		2
4	414652.60	Axle for central brake 740 mm		1
5	414653.60	Axle for central brake 700 mm		1
6	414654	Retainer for central brake		2
7	803279	Starlock button Ø10 mm		4
8	803876	Spring washer M8		4
9	804239	Nylon washer Ø13 x Ø24 x 2.5 mm		4
10	806644	Spiral clamp pin Ø5 x 14 mm		2
11	812875	Screw CBHc M8 x 12 mm 10.9		4
12	817243	Screw M6 x 8 mm		2
13	1423911	Grey cap Ø32 mm		4
14	819460	Castor Ø100 mm for central brake		3
14	1423988	Castor Ø150 Tente, with central brake		3
14	1423937	Castor Ø125 mm with central brake		3
15	1493989	Castor Ø75 mm with brake colson		1
15	819459	Castor Ø100 Tente, with central brake and dir. control		1
15	1423938	Castor Ø125 Tente, with central brake and dir. control		1
16	1524937-0154	Not for sepaeate sale, please contact Invacare		1
17	1524938-0154	Not for sepaeate sale, please contact Invacare		1
18	1451305-0154	Non-dividable base		1
19	1484312	Label for brake and direction control, 40x80, left		1
20	1484313	Label for brake and direction control, 40x80, right		1

Low slung kit

Pos.	Part number	Description	Valid from → until	Qty
-		Ø75 mm low slung kit		1


Ø75 mm low slung kit

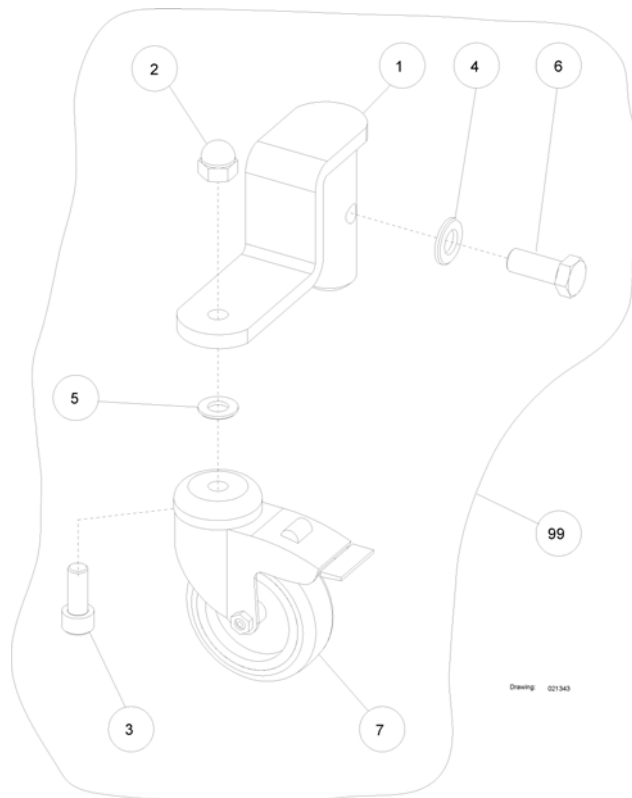


Pos.	Part number	Description	Valid from → until	Qty
1	320880-0154	Castor suspension for low slung kit		1
2	802744	Nut M10		1
3	803056	Screw M10x25 St. 8.8 ISO 4762		1
4	803273	Washer ISO7090 M12 zn		1
5	803847	Washer M10, ISO7090		1
6	806637	Screw M12x30		1
7	6790023	Castor Ø75 Tente, with brake pedal		1
99	021343-0154	Complete lowering-kit		1



Susanne bed end

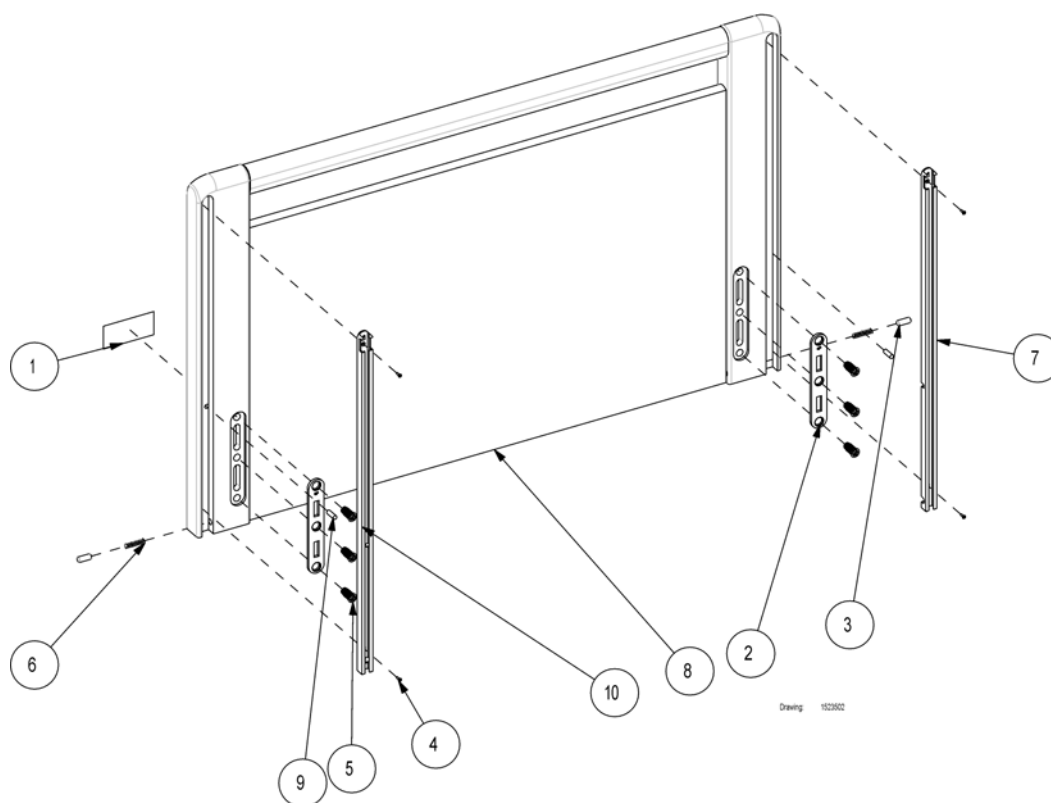


Pos.	Part number	Description	Valid from → until	Qty
1	417547	Label with Invacare logo		1
2	418441	Bracket with holes		2
3	420612.60	Pin Ø8x22		2
4	812847	Screw, ø2,5x10		4
5	818338	Screw M8x24, special		6
6	819444	Spring, ø0,75x6x30 (no.1307)		2
7	1523103	Rail for side rail, left		1
7	1524332	Rail for side rail, left, for Susanne increased		1
8	1505835	Not for separate sale - please contact Invacare		0
9	1523503-0002	Pin, ø6x20 ISO2338		2
10	1524333	Rail for side rail, right, for Susanne increased		1
10	1523824	Rail for side rail, right		1

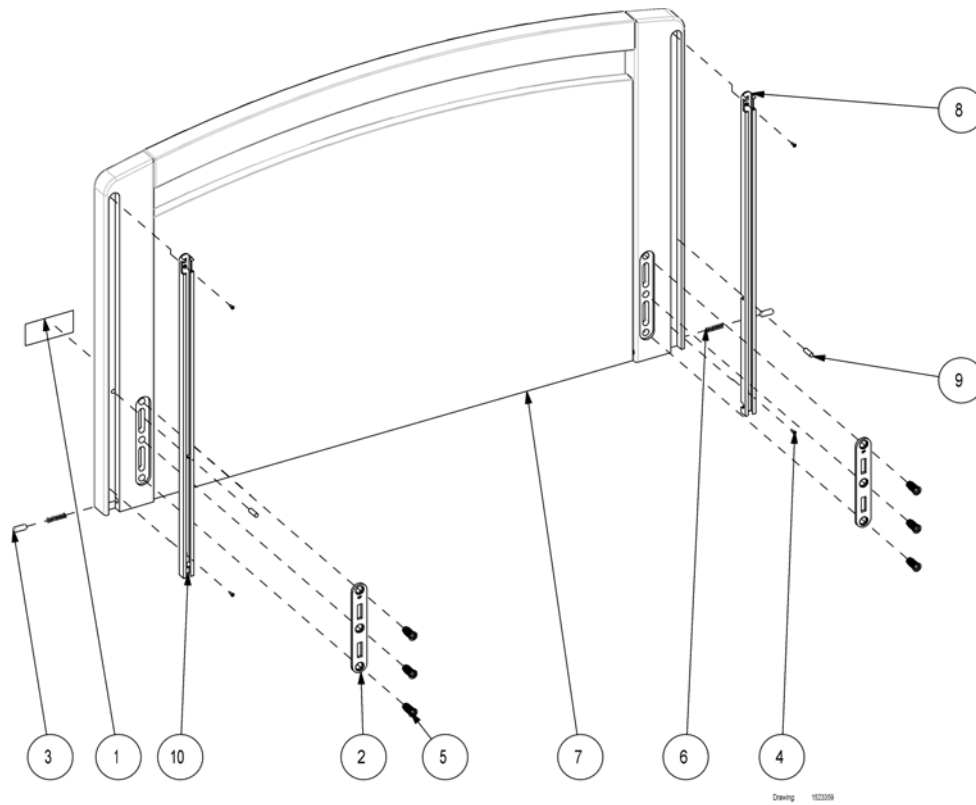Sophie bed end


Pos.	Part number	Description	Valid from → until	Qty
1	417547	Label with Invacare logo		1
2	418441	Bracket with holes		2
3	420612.60	Pin Ø8x22		2
4	812847	Screw, ø2,5x10		4
5	818338	Screw M8x24, special		6
6	819444	Spring, ø0,75x6x30 (no.1307)		2
7	1505835	Not for separate sale - please contact Invacare		0
8	1523103	Rail for side rail, left		1
9	1523503-0002	Pin, ø6x20 ISO2338		2
10	1523824	Rail for side rail, right		1

Emma bed end

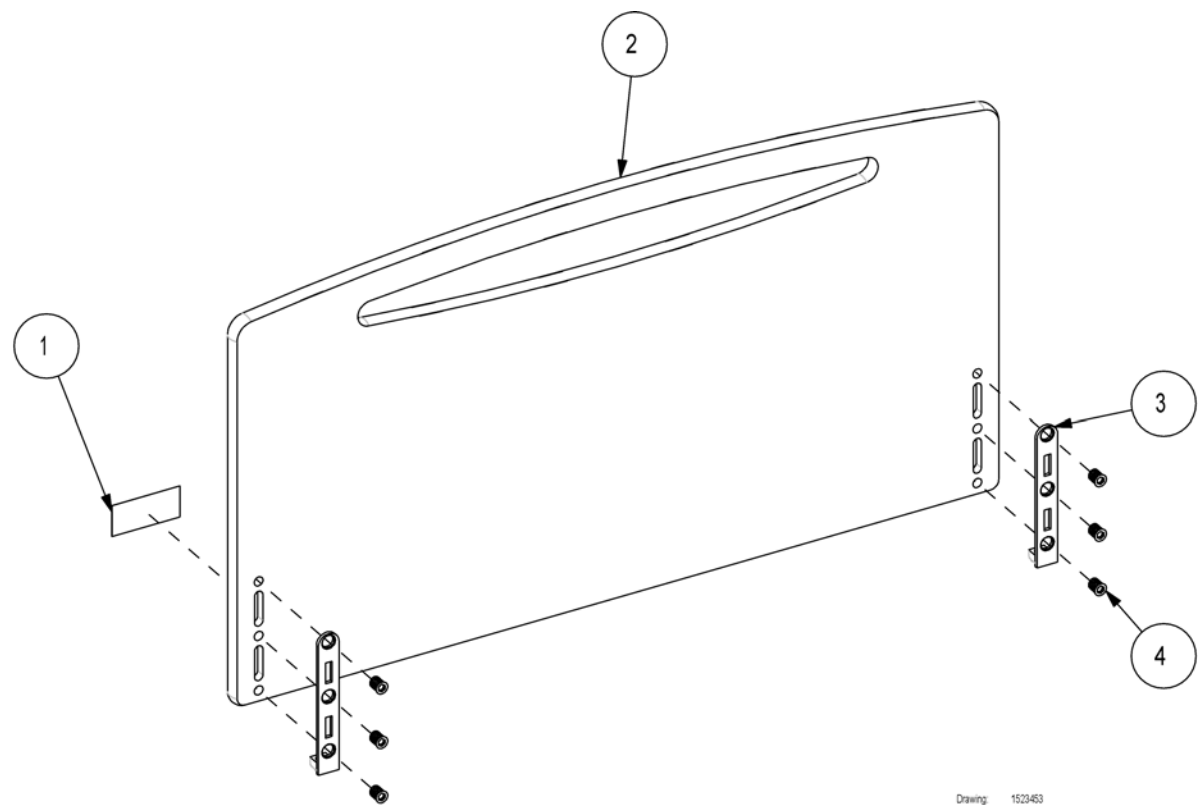


Drawing: 1523453

Pos.	Part number	Description	Valid from → until	Qty
1	417547	Label with Invacare logo		1
2	1505835	Not for separate sale - please contact Invacare		1
3	1523676-0002	Fitting with nut		2
4	1523784	Screw M8x17 special		6

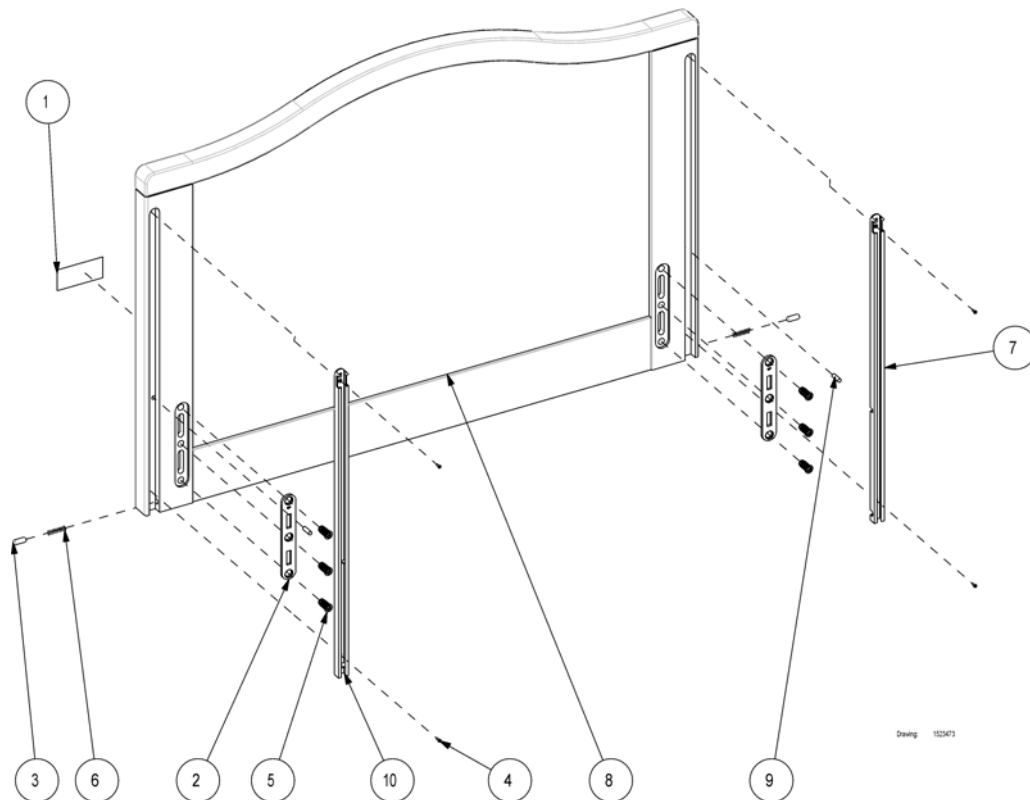Anita bed end

Pos.	Part number	Description	Valid from → until	Qty
1	417547	Label with Invacare logo		1
2	418441	Bracket with holes		2
3	420612.60	Pin Ø8x22		2
4	812847	Screw, Ø2,5x10		4
5	818338	Screw M8x24, special		6
6	819444	Spring, Ø0,75x6x30 (no.1307)		2
7	1523103	Rail for side rail, left		1
8	1505835	Not for separate sale - please contact Invacare		0
9	1523503-0002	Pin, Ø6x20 ISO2338		2
10	1523824	Rail for side rail, right		1

Victoria bed end

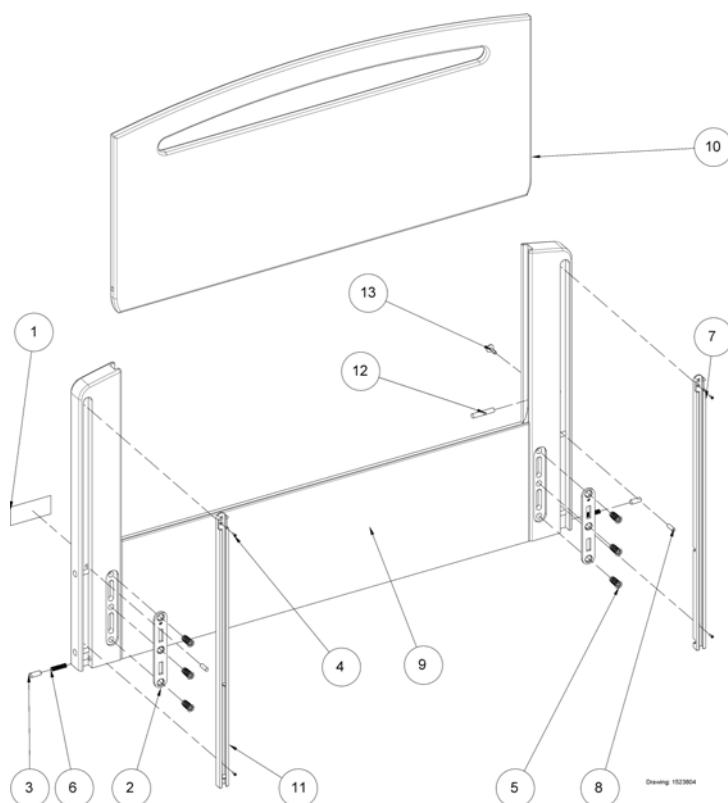


Drawing 1523804

Pos.	Part number	Description	Valid from → until	Qty
1	417547	Label with Invacare logo		1
2	418441	Bracket with holes		2
3	420612.60	Pin Ø8x22		2
4	812847	Screw, Ø2,5x10		4
5	818338	Screw M8x24, special		6
6	819444	Spring, Ø0,75x6x30 (no.1307)		3
7	1523103	Rail for side rail, left		1
8	1523503-0002	Pin, Ø6x20 ISO2338		2
9	1523790-0101	Bed end, Victoria		1
10	1523807-0101	Plate for Victoria		1
11	1523824	Rail for side rail, right		1
12	1529563-0002	Locking pin, Ø8x45		1
13	1529564	Release button, Ø14 M5		1

Motor for back rest

Pos.	Part number	Description	Valid from → until	Qty
-		Motor for back rest		1

Motor for back rest

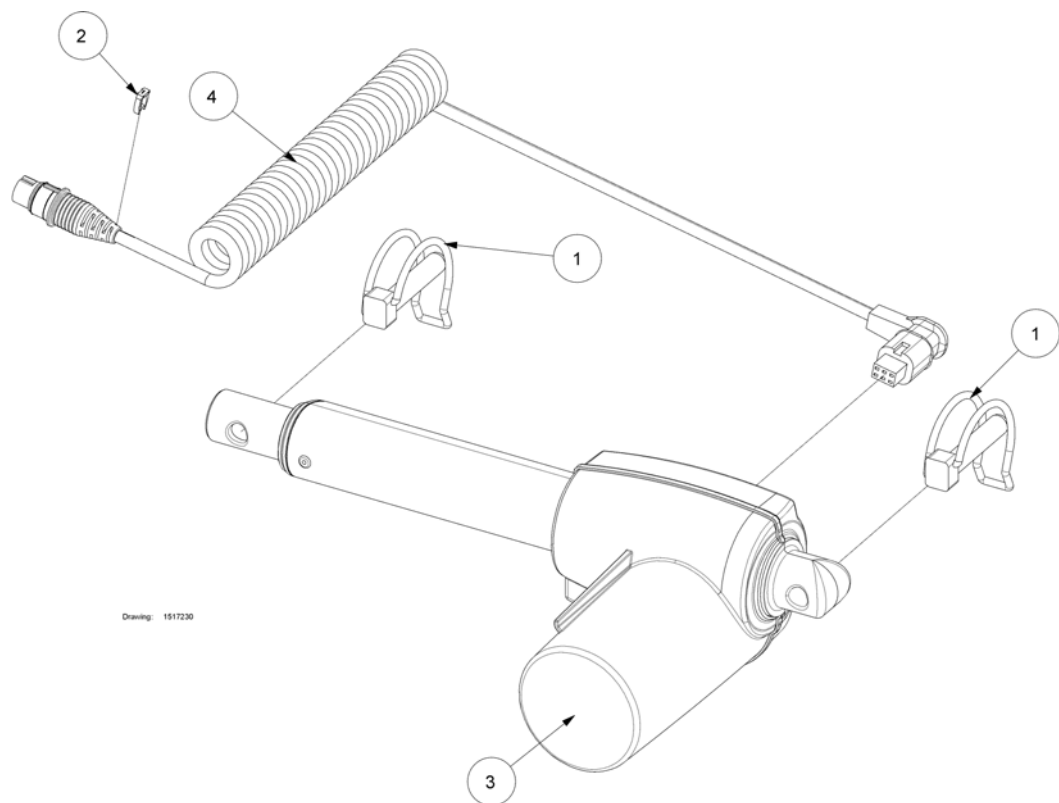


Pos.	Part number	Description	Valid from → until	Qty
1	819411	Safety pin Ø10 x 50 mm		2
2	1444092	Cable marker 1, brown, for backrest motor		1
3	1514926-7035	Motor for back rest, LA27		1
4	1516912	Cable 200mm with coil for LA27/CB09, SB755		1

Motor for thigh section

Pos.	Part number	Description	Valid from → until	Qty
-		Motor for thigh section		1

Motor for thigh section

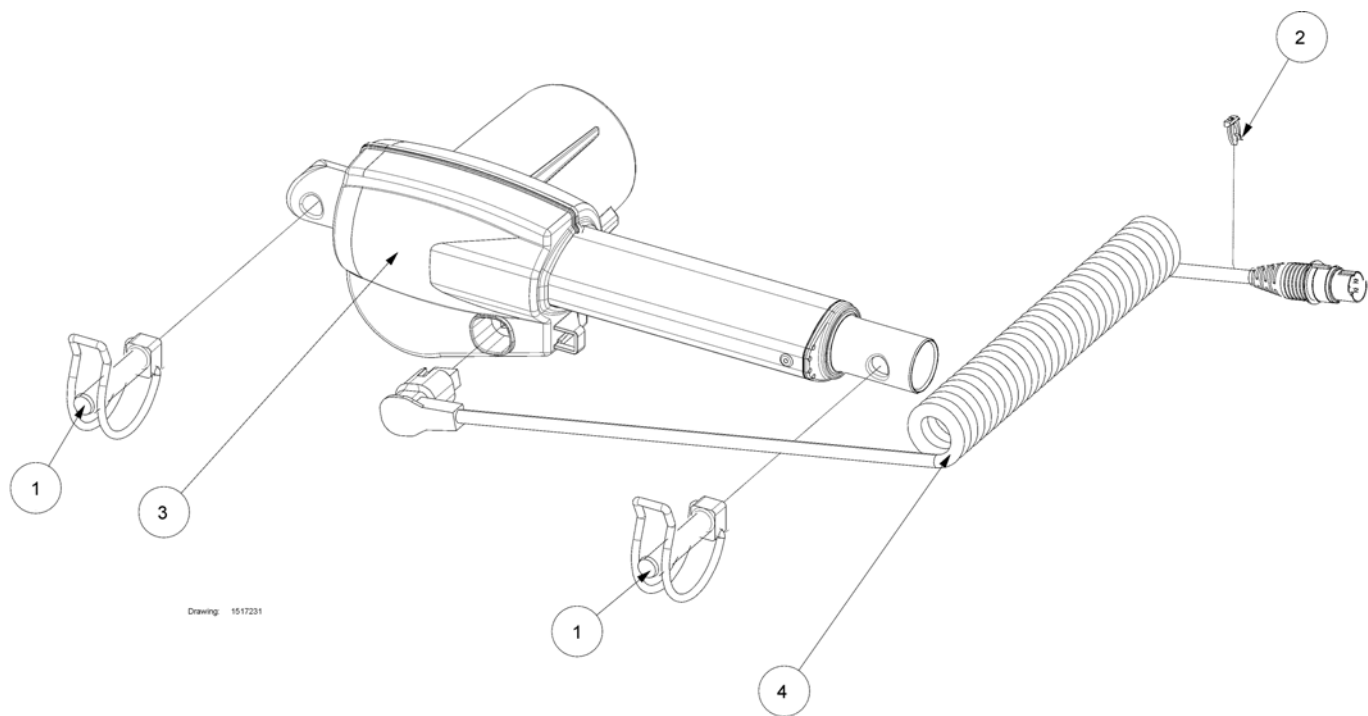


Pos.	Part number	Description	Valid from → until	Qty
1	819411	Safety pin Ø10 x 50 mm		2
2	1444094	Cable marker 3, orange, for thigh motor		1
3	1514927-7035	Motor for thigh section, LA27		1
4	1516912	Cable 200mm with coil for LA27/CB09, SB755		1

Motor for leg section

Pos.	Part number	Description	Valid from → until	Qty
-		Motor for leg section		1

Motor for leg section

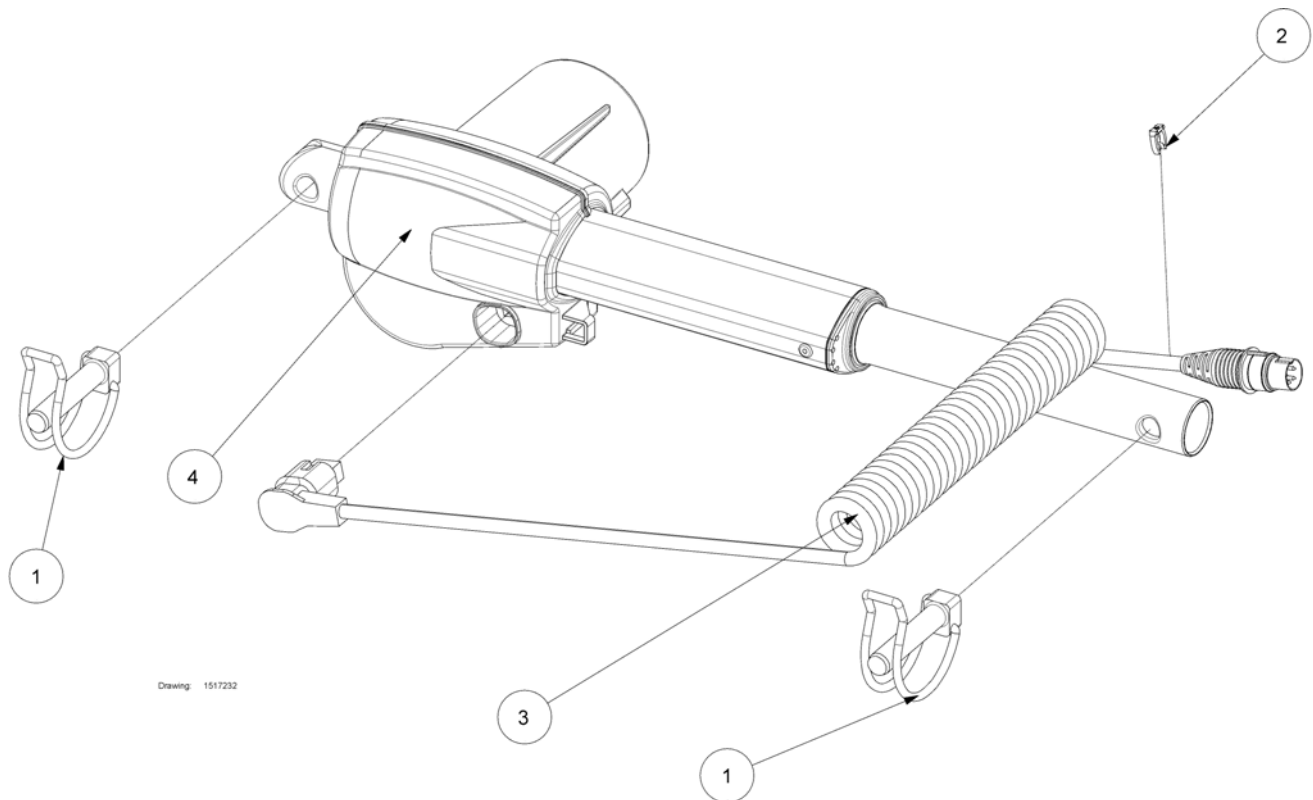


Pos.	Part number	Description	Valid from → until	Qty
1	819411	Safety pin Ø10 x 50 mm		2
2	1444095	Cable marker 4, yellow, for leg motor		1
3	1516912	Cable 200mm with coil for LA27/CB09, SB755		1
4	1523118-7035	Motor for leg section, LA27		1


Motor for height adjustment

Pos.	Part number	Description	Valid from → until	Qty
-		Motor for height adjustment		1
-		Motor for height adjustment, 120 cm width		1

Motor for height adjustment

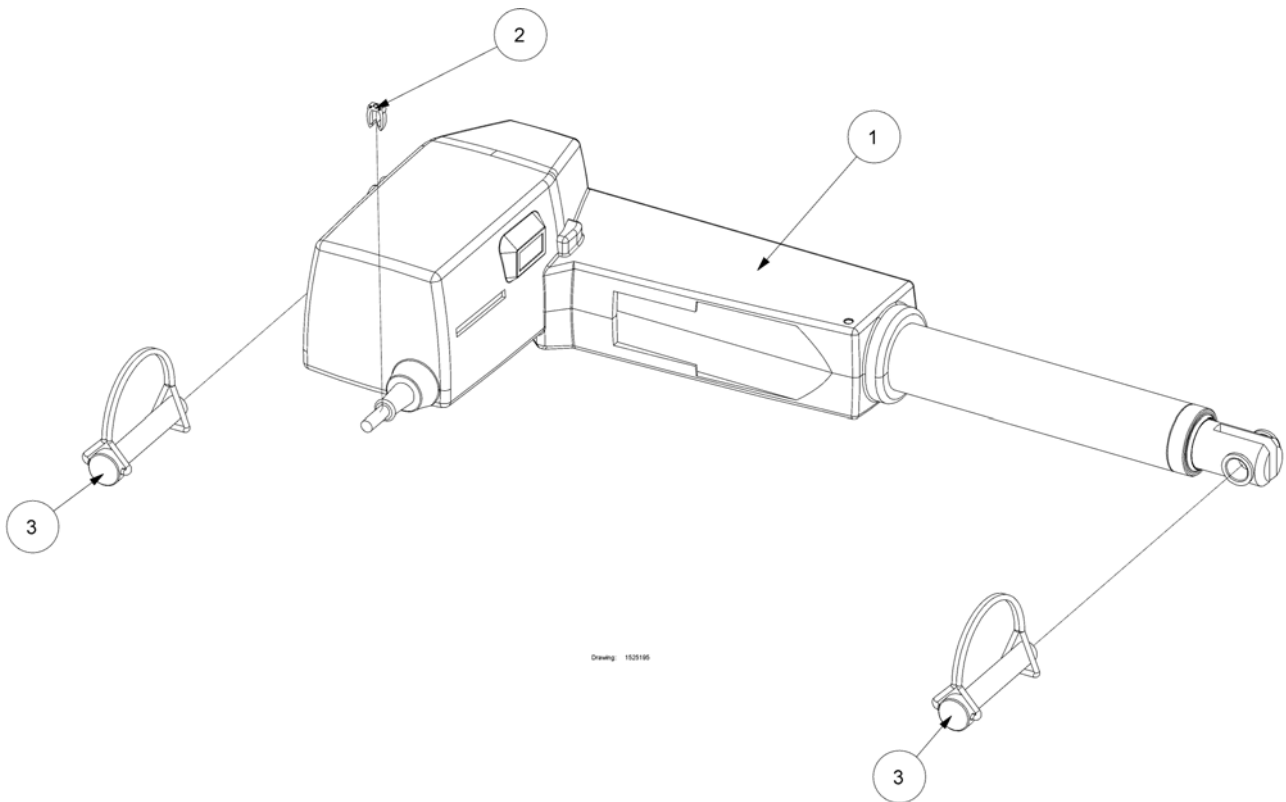

Pos.	Part number	Description	Valid from → until	Qty
1	1423901	24V motor LA31, IP66, RAL 7035		1
2	1444093	Cable marker 2, red, for height motor		1
3	1507121	Shaft locking pin Ø10x50		2

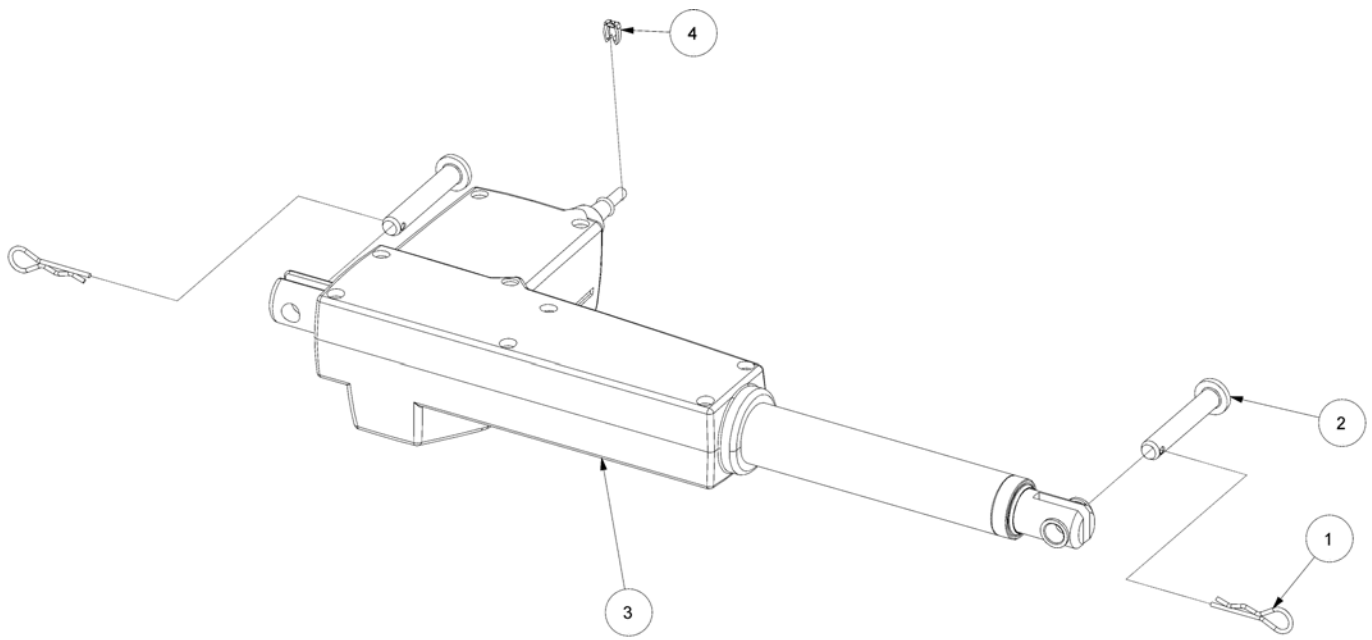
Motor for height adjustment, 120 cm width

Drawing: 021370

Pos.	Part number	Description	Valid from → until	Qty
1	816427	Spring pin Ø2x43		2
2	821204	Clevis pin Ø10x60 ISO2341		2
3	1423909	24V motor LA31, IP66, RAL 7035		1
4	1444093	Cable marker 2, red, for height motor		1


Control unit

Pos.	Part number	Description	Valid from → until	Qty
-		Control unit without accumulator backup, 4 motors, EU		1
-		Control unit without accumulator backup, 4 motors, UK		1
-		Control unit prepared for accumulator backup, 4 motors, EU		1
-		Control unit prepared for accumulator backup, 4 motors, UK		1

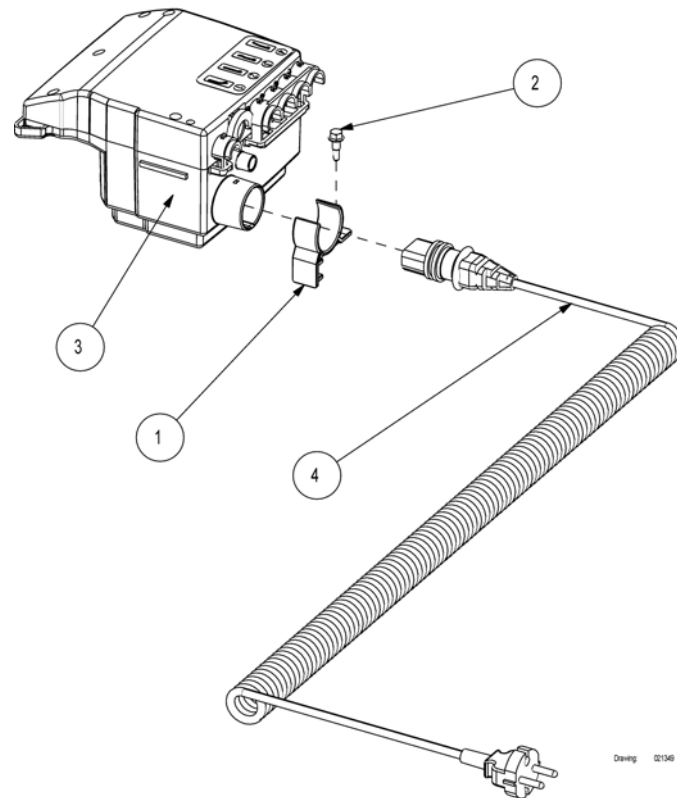Control unit without accumulator backup, 4 motors, EU


Pos.	Part number	Description	Valid from → until	Qty
1	320522-7035	Bracket for control unit CB09		1
2	819407	Screw Ø5,5x19 ISO 15480		1
3	1423906	Control unit CB09		1
4	1424000	Power cable EU		1

Control unit without accumulator backup, 4 motors, UK

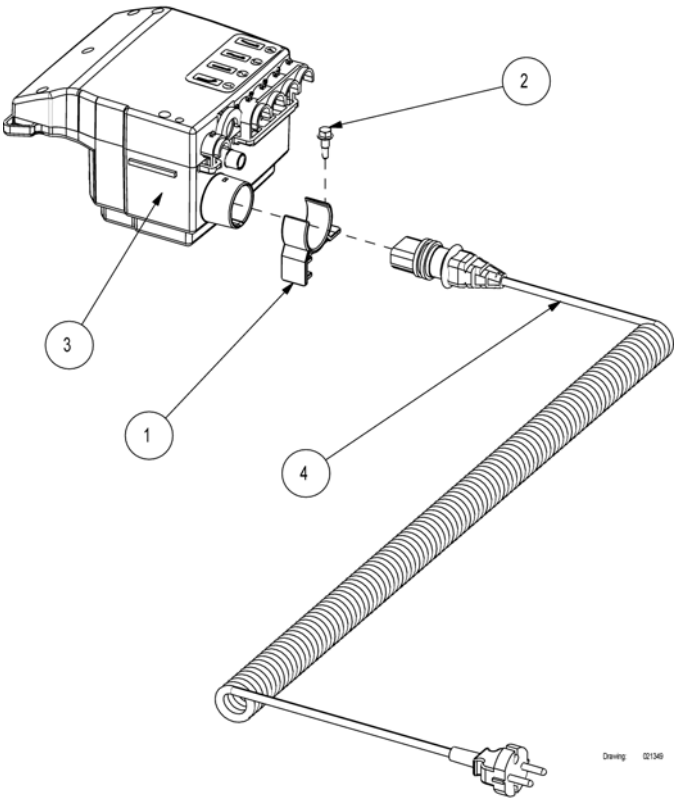


Pos.	Part number	Description	Valid from → until	Qty
1	320522-7035	Bracket for control unit CB09		1
2	819407	Screw Ø5,5x19 ISO 15480		1
3	1423906	Control unit CB09		1
4	819430	Power cable UK		1

Control unit prepared for accumulator backup, 4 motors, EU

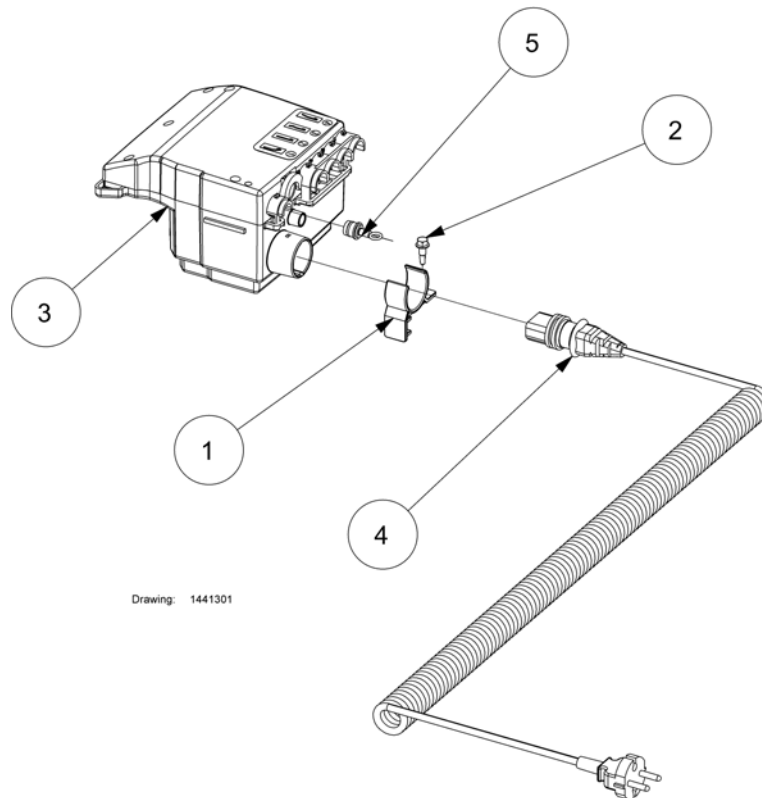


Pos.	Part number	Description	Valid from → until	Qty
1	320522-7035	Bracket for control unit CB09		1
2	819407	Screw Ø5,5x19 ISO 15480		1
3	1423907	Control unit CB09 for accumulator backup		1
4	1424000	Power cable EU		1
5	1441299-7035	Plug for control unit CB09		1

Control unit prepared for accumulator backup, 4 motors, UK

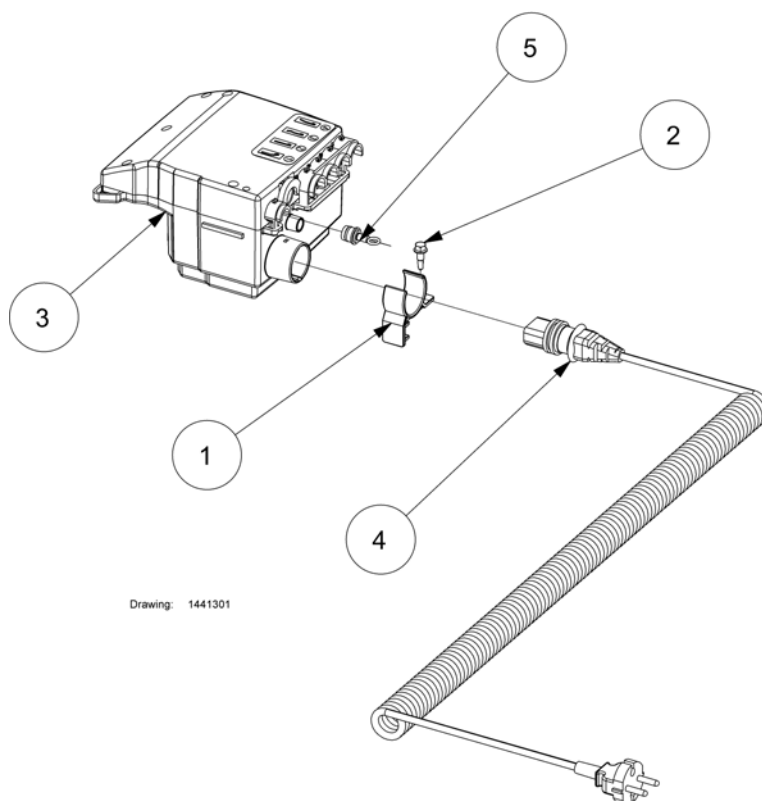


Pos.	Part number	Description	Valid from → until	Qty
1	320522-7035	Bracket for control unit CB09		1
2	819407	Screw Ø5,5x19 ISO 15480		1
3	1423907	Control unit CB09 for accumulator backup		1
4	819430	Power cable UK		1
5	1441299-7035	Plug for control unit CB09		1

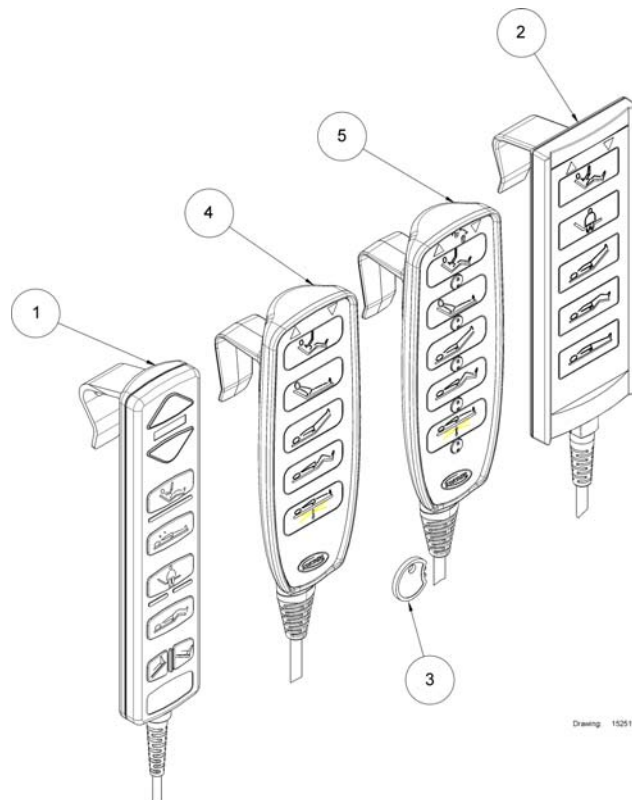
Hand control

Pos.	Part number	Description	Valid from → until	Qty
-	-	Extension cable and Y-cable		1
1	1423983	Soft control 4 motors for CB09		1
2	1492876	Hand control 4 motors for CB09		1
3	1493529	Blocking key for hand control HL80		1
4	1531345-7035	Hand control, HB80 without key		1
5	1531346-7035	Hand control, HL80 with key		1

Extension cable and Y-cable

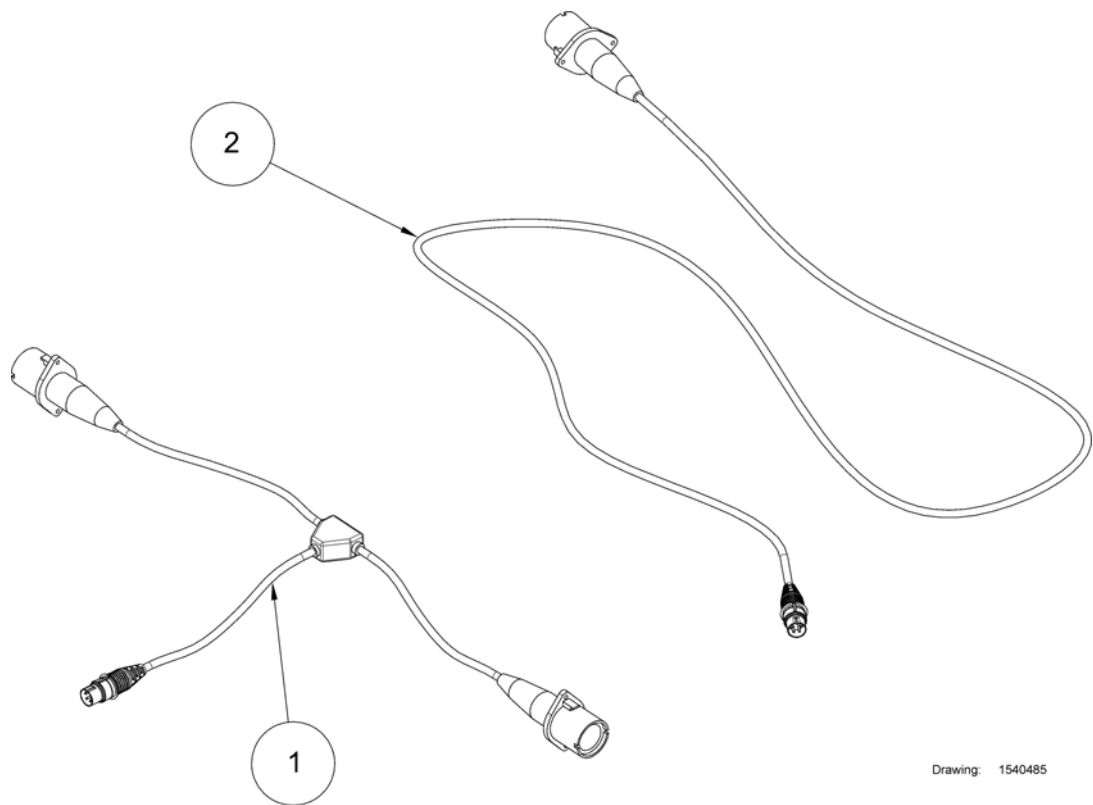


Pos.	Part number	Description	Valid from → until	Qty
1	819465	Y-cable		1
2	1532489-7035	Extension cable		1

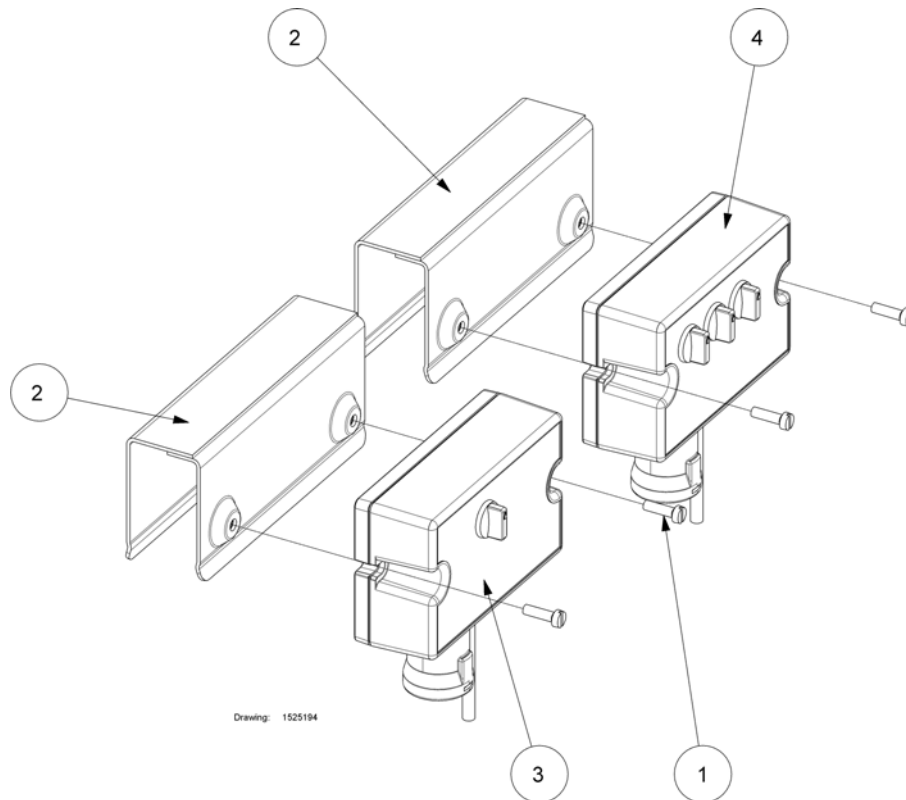
ACP box

Pos.	Part number	Description	Valid from → until	Qty
1	812358	Screw M4x14 ISO1207		2
2	1525446-0154	Bracket for ACP box, SB755		1
3	1435255	ACP box, micro		1
4	1450827	ACP box		0

Battery backup

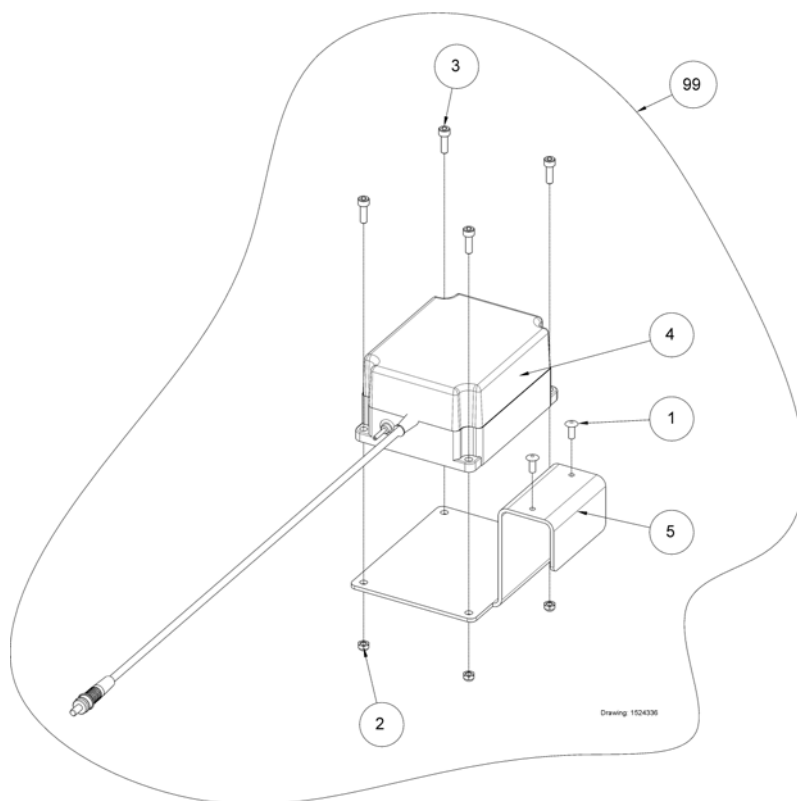


Pos.	Part number	Description	Valid from → until	Qty
1	803833	Rivet Ø4,8x12		2
2	804458	Nut M5 ISO 10511, self locking		4
3	816480	Screw M5x16 ISO 4752		4
4	1423905	Accumulator backup unit for CB09		1
5	1523785-0154	Bracket for backup battery		1
99	1524336-0154	Battery backup		1

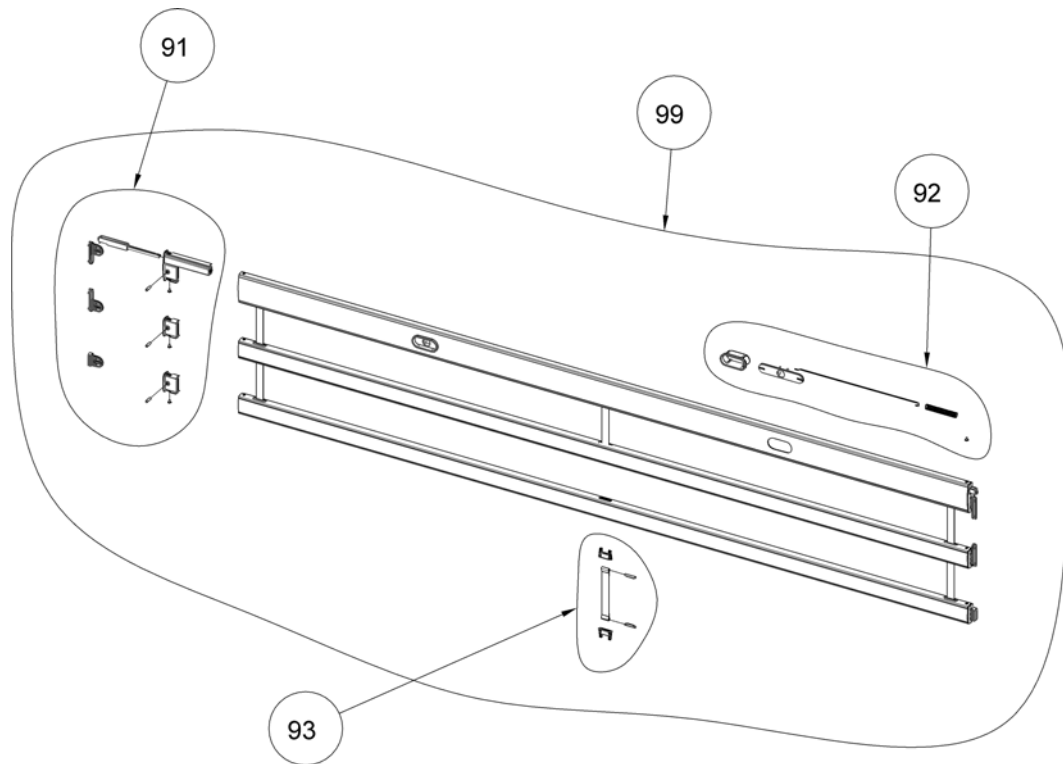Line siderail

Drawing: 1522786_IL


Pos.	Part number	Description	Valid from → until	Qty
91	1547315	Spare parts kit 1, sliding parts		1
92	1547316	Spare parts kit 2, lock and wire		1
93	1525497	Strap with fittings, for Line side rail		1
99	1522786-0125	Side-rails Alu Line Beech (pair)		1
99	1522786-0126	Line side rail, 200 cm, white pigmented		1
99	1522786-0127	Side-rails Alu Line cherry (pair)		1

Line siderail extendable

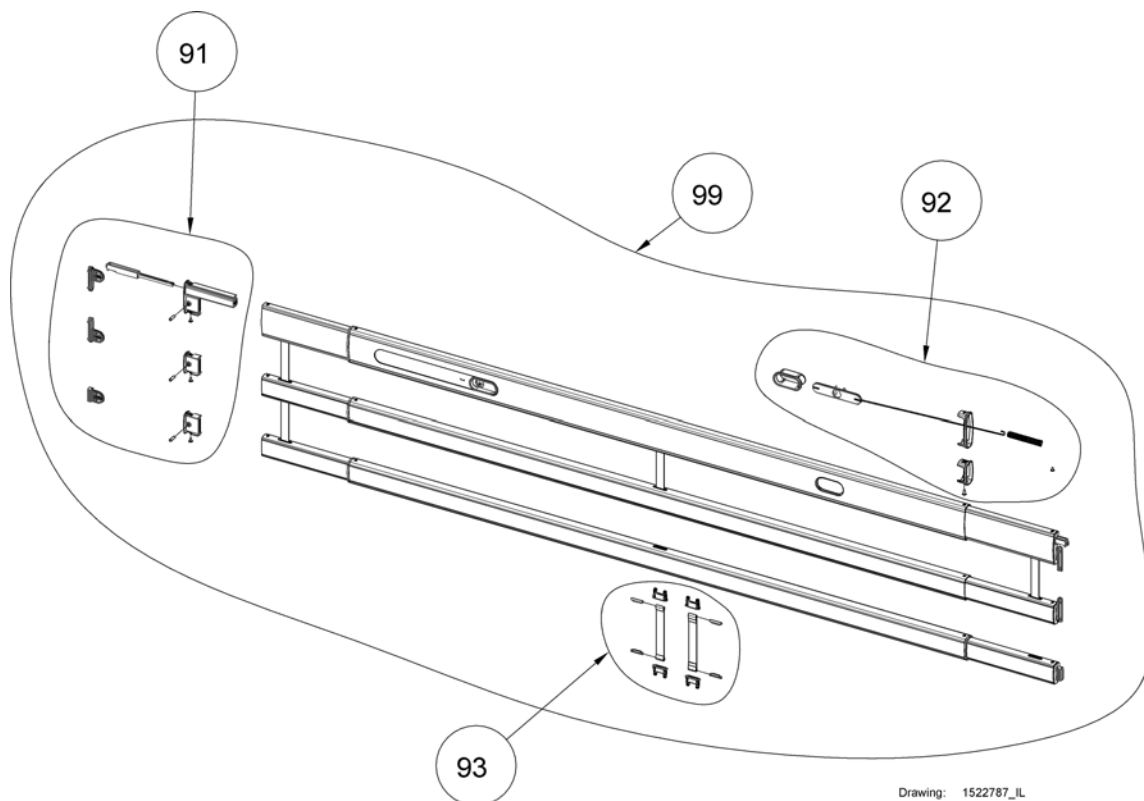

Drawing: 1522787_IL


Pos.	Part number	Description	Valid from → until	Qty
91	1547315	Spare parts kit 1, sliding parts		1
92	1547317	Spare parts kit 2, lock and wire		1
93	1525496	Spare parts kit 3, short strap with fittings		1
93	1525497	Strap with fittings, for Line side rail		1
99	1522787-0125	Line side rail extendable, beech		1
99	1522787-0126	Line side rail extendable, white pigmented		1
99	1522787-0127	Line side rail extendable, cherry		1

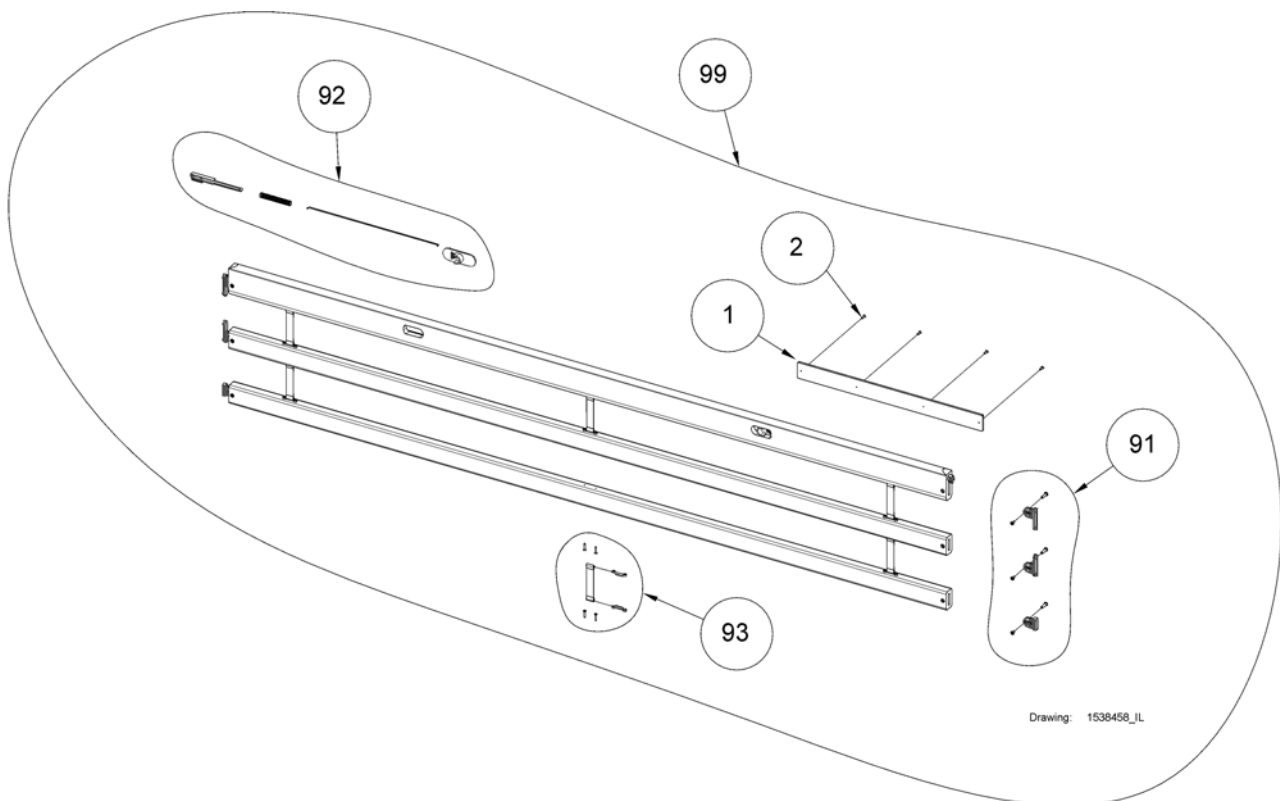Britt V side rail


Pos.	Part number	Description	Valid from → until	Qty
1	1538460-0101	Wood cover piece, Britt V, beech		2
1	1538460-0102	Wood cover piece, Britt V, cherry		2
1	1538460-0105	Wood cover piece, Britt V, white pigmented		2
2	816449	Screw, Ø3,5x15		8
91	1552606	Sliding kit for side rail Britt V		1
92	1552607	Release system kit for wooden side rails Britt V		1
93	1552608	Spare parts kit 3, strap w/ fittings (1 side rail)		1
99	1538458-0101	Side rail, Britt V, beech (1 pair)		1
99	1538458-0102	Side rail, Britt V, cherry (1 pair)		1
99	1538458-0105	Side rail, Britt V, white pigmented (1 pair)		1

Scala steel side rail


Pos.	Part number	Description	Valid from → until	Qty
-		Scala Basic 2, side rail		1
-		Scala Medium 2, side rail		1
-		Scala Decubi 2, side rail		1
-		Scala Basic Plus 2, side rail		1

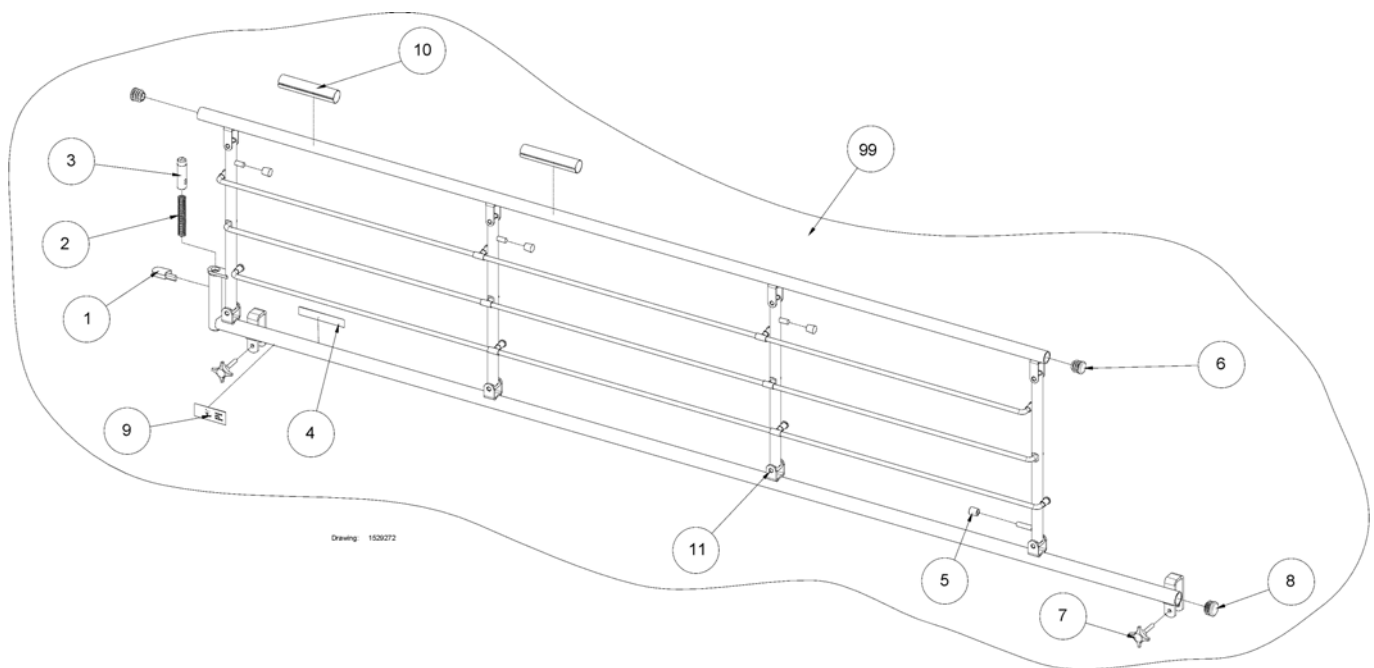


Pos.	Part number	Description	Valid from → until	Qty
1	314291-7035	Release button for steel side rail		1
2	414465	Spring Ø13,5x75xØ1,5		1
3	414965-7035	Ratchet for release button		1
4	416303	Type badge 15.9x88.9 white (silver-poly)		1
5	804386-7035	Ferrule ø8		4
6	804399-7035	End plug Ø22		2
7	818388-7035	Finger screw M6x35		2
8	819424-7035	Part not for sale		1
9	1415774	Label for positioning of side rail		0
10	1435100	Label for placement of hands, metal side rails		2
11	1529270	Part not for sale		0
11	1528975	Part not for sale		0
99	1529272-0154	Scala Basic 2, side rail right		1
99	1528931-0154	Scala Basic 2, side rail left		1

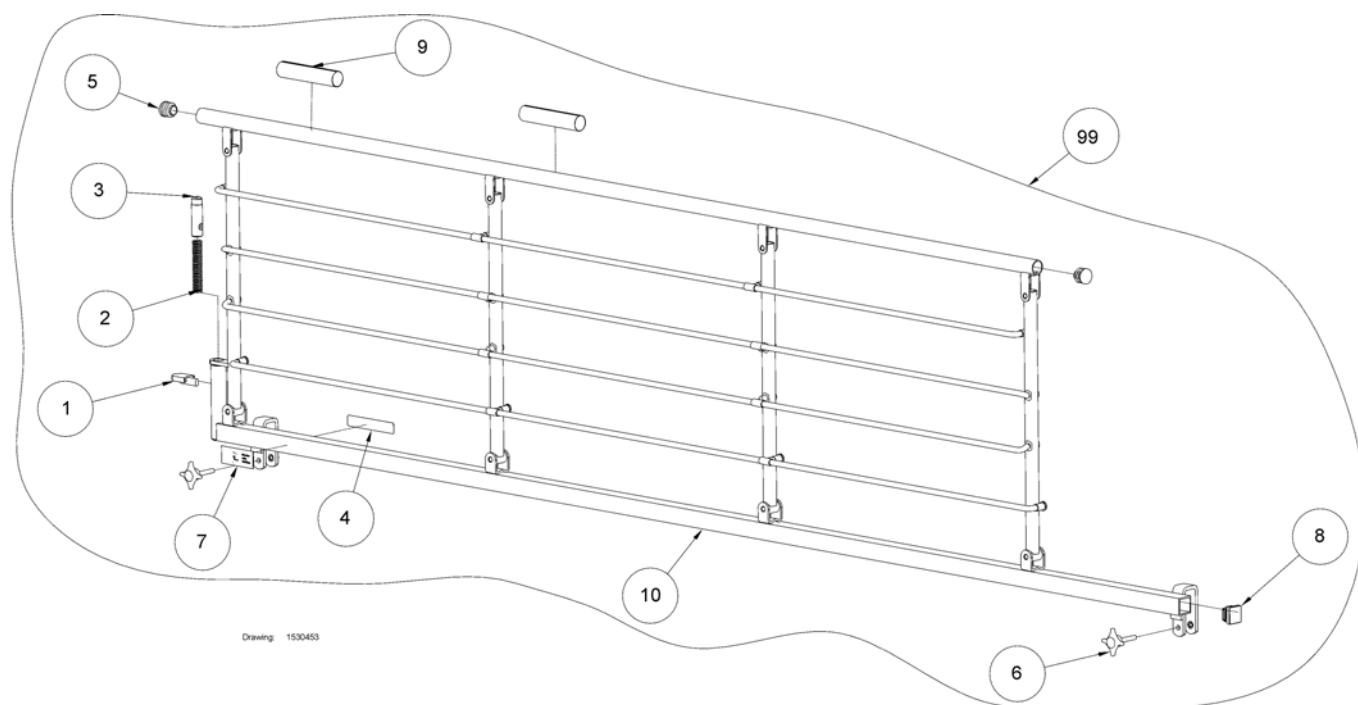Scala Medium 2, side rail


Pos.	Part number	Description	Valid from → until	Qty
1	314291-7035	Release button for steel side rail		1
2	414465	Spring Ø13,5x75xØ1,5		1
3	414965-7035	Ratchet for release button		1
4	416303	Type badge 15.9x88.9 white (silver-poly)		1
5	804399-7035	End plug Ø22		2
6	818388-7035	Finger screw M6x35		2
7	1415774	Label for positioning of side rail		0
8	1423912-7035	Ferrule 25x25		1
9	1435100	Label for placement of hands, metal side rails		2
10	1530454	Part not for sale		0
10	1530438	Part not for sale		0
99	1530453-0154	Scala Medium 2, side rail right		1
99	1530437-0154	Scala Medium 2, side rail left		1

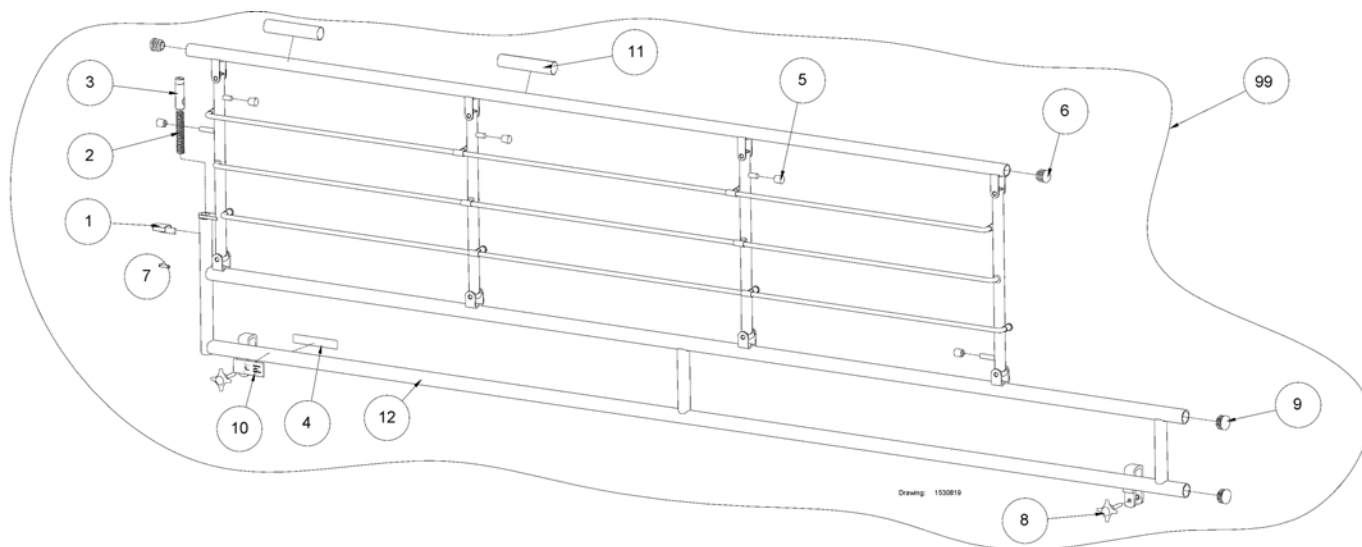Scala Decubi 2, side rail

Pos.	Part number	Description	Valid from → until	Qty
1	314291-7035	Release button for steel side rail		1
2	414465	Spring Ø13,5x75xØ1,5		1
3	414965-7035	Ratchet for release button		1
4	416303	Type badge 15.9x88.9 white (silver-poly)		1
5	804386-7035	Ferrule ø8		4
6	804399-7035	End plug Ø22		2
7	806790	Spiral clamp pin ø5x20		1
8	818388-7035	Finger screw M6x35		2
9	819424-7035	Part not for sale		2
10	1415774	Label for positioning of side rail		0
11	1435100	Label for placement of hands, metal side rails		2
12	1530822	Part not for sale		0
12	1530820	Part not for sale		0
99	1530819-0154	Scala Decubi 2, side rail right		1
99	1530818-0154	Scala Decubi 2, side rail left		1

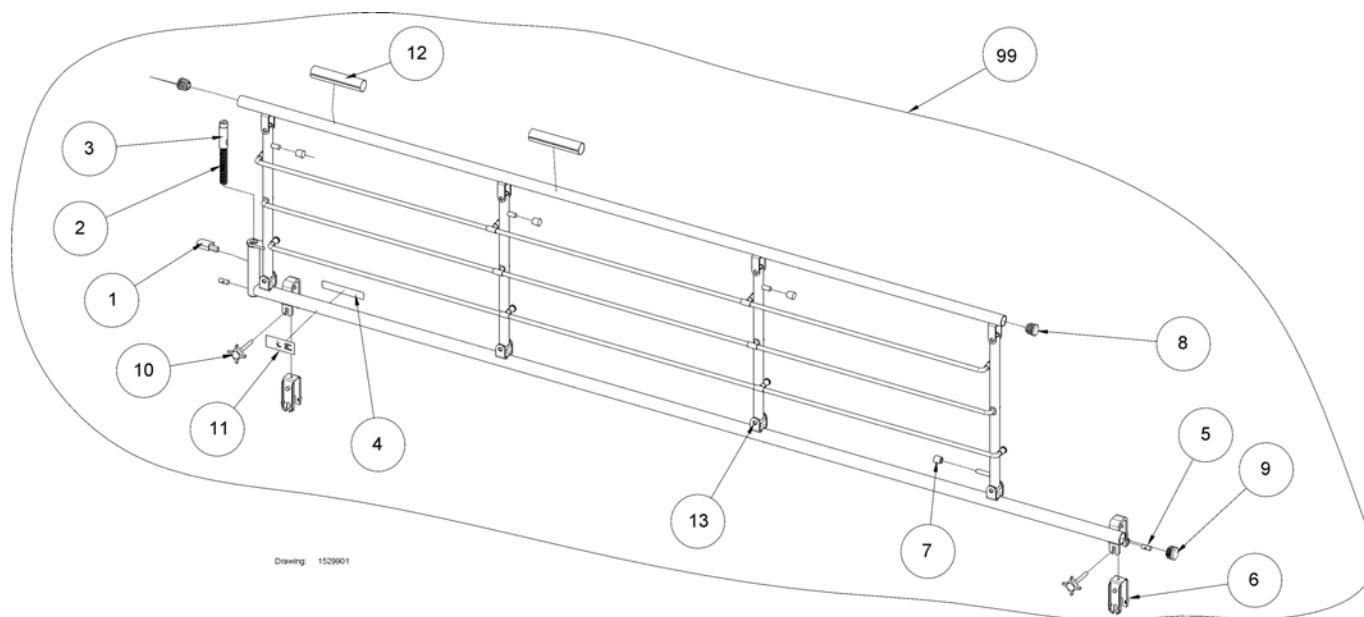
Scala Basic Plus 2, side rail

Pos.	Part number	Description	Valid from → until	Qty
1	314291-7035	Release button for steel side rail		1
2	414465	Spring Ø13,5x75xØ1,5		1
3	414965-7035	Ratchet for release button		1
4	416303	Type badge 15.9x88.9 white (silver-poly)		1
5	421468	Cylinder nut for u-bow		2
6	421843-7035	Inlay for u-brace, steel side rail		2
7	804386-7035	Ferrule ø8		4
8	804399-7035	End plug Ø22		2
9	819424-7035	Part not for sale		1
10	821251-7035	Finger screw M6x45		2
11	1415774	Label for positioning of side rail		1
12	1435100	Label for placement of hands, metal side rails		2
13	1529905	Part not for sale		0
13	1529904	Part not for sale		0
99	1529901-0154	Scala Basic Plus 2, side rail right		1
99	1529746-0154	Scala Basic Plus 2, side rail left		1

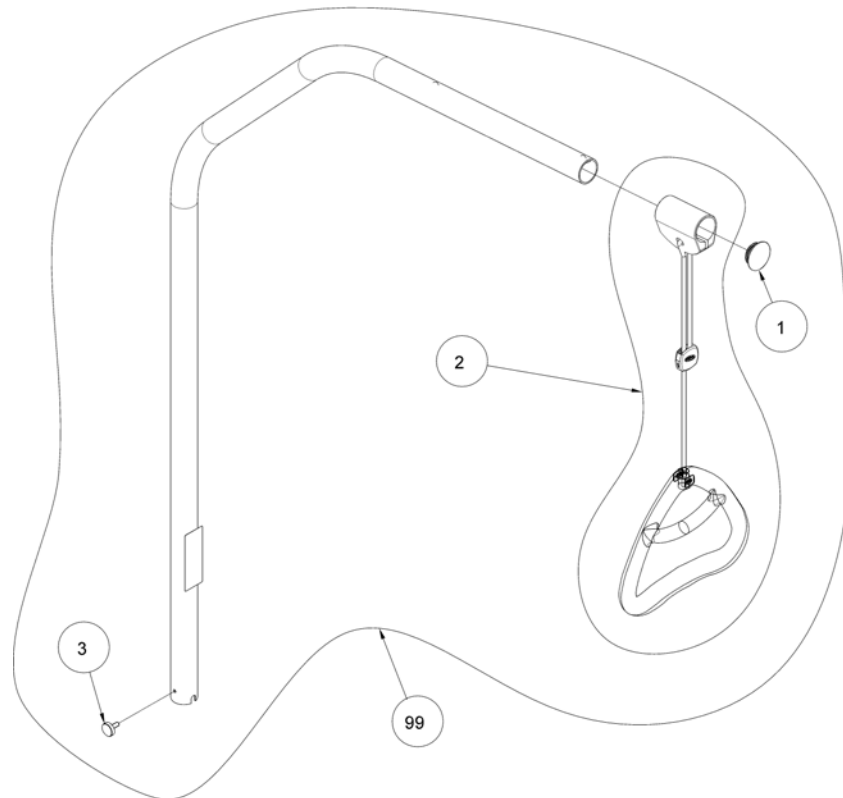Lifting pole, Carebed, light


Pos.	Part number	Description	Valid from → until	Qty
1	317836	Cap for lifting pole		1
2	320597	Lifting pole handle		1
3	803979-9005	Finger screw M6 x 16 mm Ø20 (lifting pole)		1
99	1542093-0154	Lifting pole, Carebed light		1

Drip rod

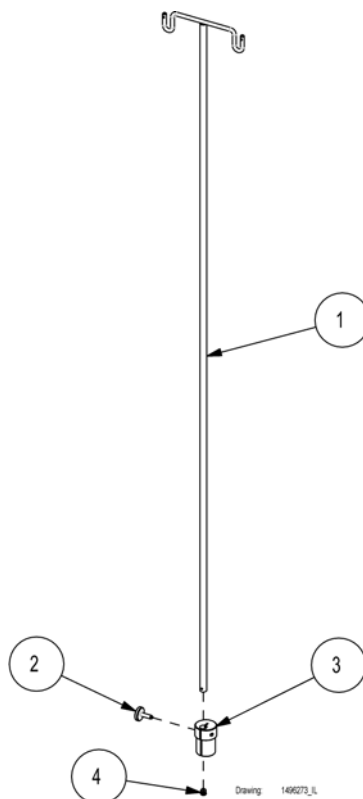


Pos.	Part number	Description	Valid from → until	Qty
1	3000610.60030P0	Drip rod		1
2	405049	Finger screw M8 x 25 mm		1
3	420681	Holder for drip rod		1
4	804409	Ferrule 12 mm black		1
5	405046	Finger screw M8x16 with soft tip		1
98	3000610.59570P0	Bracket for IV drip rod		1
99	50.60910.00	Bush for IV drop rod		1

Bumper wheel

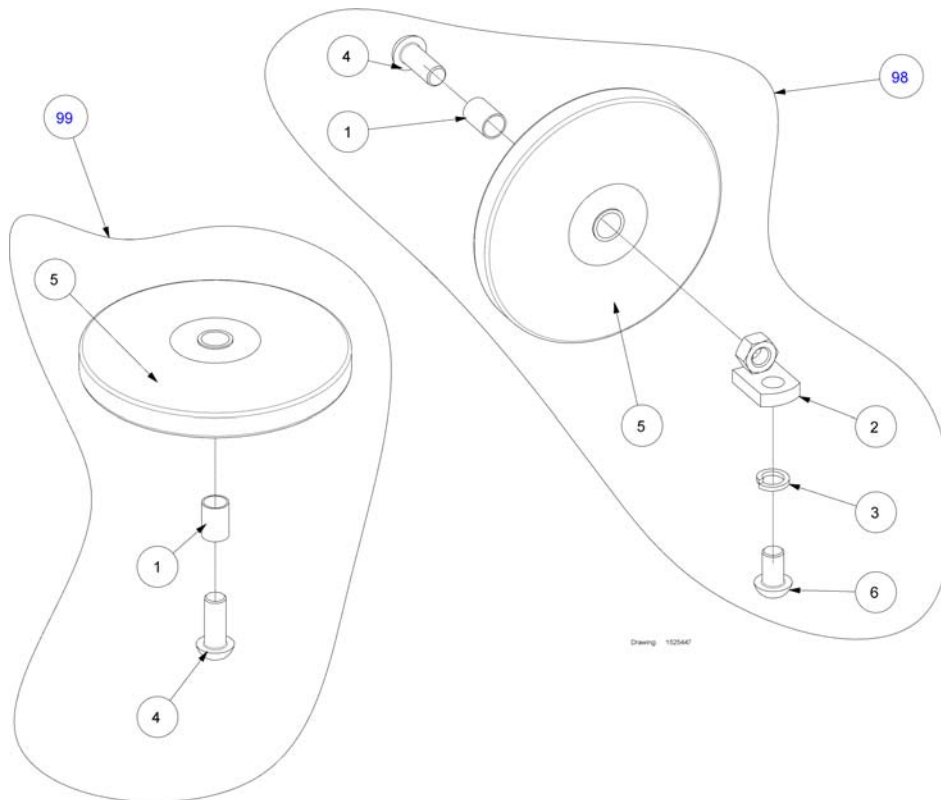

Pos.	Part number	Description	Valid from → until	Qty
1	405656	Bushing Ø15x1x21		1
2	414695-0154	Fittings for bumper wheel		1
3	803275	Spring washer M12 DIN 7980		1
4	810871	Screw M12x30 ISO 7380 St.10.9		1
5	814310	Bumper wheel Ø100/Ø15,5 RAL 7001		1
5	409174	Bumper wheel Ø75/Ø15,5 RAL 7001		1
5	816436	Bumper wheel Ø150/Ø15,5 RAL 7001		1
6	814399	Screw M12x20 St. 10.9		1

Support handle

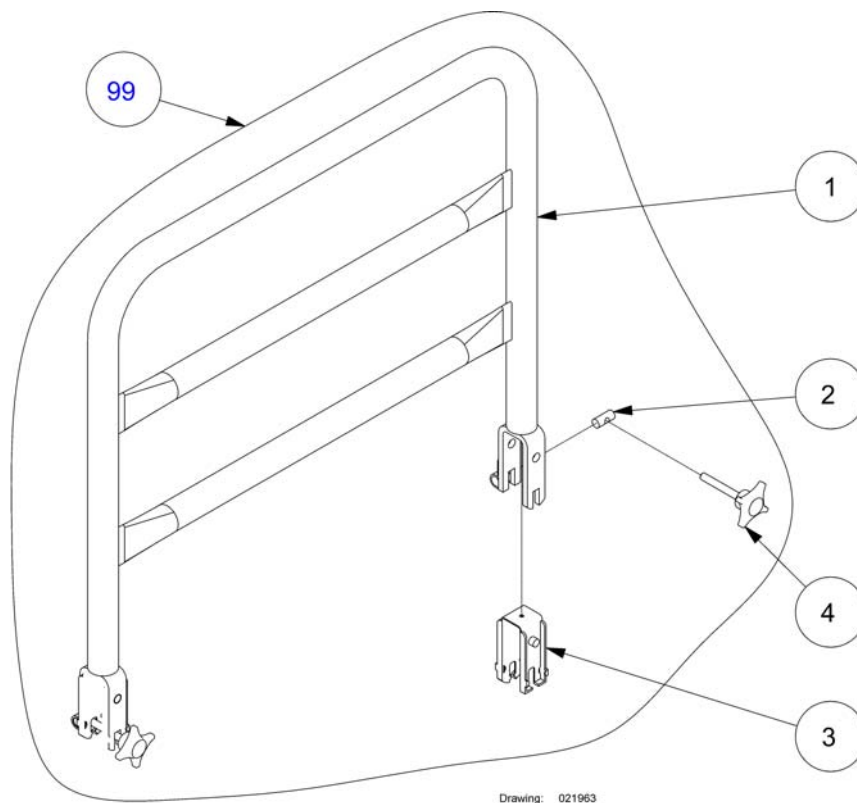

Pos.	Part number	Description	Valid from → until	Qty
1	1505835	Not for separate sale - please contact Invacare		21
2	421468	Cylinder nut for u-bow		2
3	421843-7035	Inlay for u-brace, steel side rail		2
4	821251-7035	Finger screw M6x45		2
99	021964-0154	Support handle 25x30 cm		1
99	021963-0154	Support handle 40x50 cm		1

Cable holder

Pos.	Part number	Description	Valid from → until	Qty
1	1550422-0154	Part not for sale		1



Yes, you can.®



Belgium & Luxemburg:

Invacare nv, Autobaan 22, B-8210 Loppem
Tel: (32) (0)50 83 10 10, Fax: (32) (0)50 83 10 11
belgium@invacare.com / www.invacare.be

Denmark:

Invacare A/S, Sdr. Ringvej 37, DK-2605 Brøndby
Tel: (45) (0)36 90 00 00, Fax: (45) (0)36 90 00 01
denmark@invacare.com / www.invacare.dk

Deutschland:

Invacare Aquatec GmbH, Alemannenstraße 10, D-88316 Isny
Tel: (49) (0)75 62 7 00 0, Fax: (49) (0)75 62 7 00 66
info@invacare-aquatec.com / www.invacare-aquatec.de

European Distributor Organisation:

Invacare, Kleiststraße 49, D-32457 Porta Westfalica
Tel: (49) (0)57 31 754 540, Fax: (49) (0)57 31 754 541
edo@invacare.com / www.invacare.eu.com

España:

Invacare SA, c/Areny s/n, Polígono Industrial de Celrà, E-17460 Celrà (Girona)
Tel: (34) (0)972 49 32 00, Fax: (34) (0)972 49 32 20
contactsp@invacare.com / www.invacare.es

France:

Invacare Poirier SAS, Route de St Roch, F-37230 Fondettes
Tel: (33) (0)2 47 62 64 66, Fax: (33) (0)2 47 42 12 24
contactfr@invacare.com / www.invacare.fr

Ireland:

Invacare Ireland Ltd, Unit 5 Seatown Business Campus, Seatown Road, Swords, County Dublin - Ireland
Tel: (353) 1 810 7084, Fax: (353) 1 810 7085
ireland@invacare.com / www.invacare.ie

Italia:

Invacare Mecc San s.r.l., Via dei Pini 62, I-36016 Thiene (VI)
Tel: (39) 0445 38 00 59, Fax: (39) 0445 38 00 34
italia@invacare.com / www.invacare.it

Nederland:

Invacare BV, Celsiusstraat 46, NL-6716 BZ Ede
Tel: (31) (0)318 695 757, Fax: (31) (0)318 695 758
nederland@invacare.com, csede@invacare.com / www.invacare.nl

Norge:

Invacare AS, Grensesvingen 9, Postboks 6230, Etterstad, N-0603 Oslo
Tel: (47) (0)22 57 95 00, Fax: (47) (0)22 57 95 01
norway@invacare.com, island@invacare.com / www.invacare.no

Österreich:

Invacare Austria GmbH, Herzog Odilostrasse 101, A-5310 Mondsee
Tel: (43) 6232 5535 0, Fax: (43) 6232 5535 4
info-austria@invacare.com / www.invacare.at

Portugal:

Invacare Lda, Rua Estrada Velha, 949, P-4465-784 Leça do Balio
Tel: (351) (0)225 1059 46/47, Fax: (351) (0)225 1057 39
portugal@invacare.com / www.invacare.pt

Sverige & Suomi:

Invacare AB, Fagerstagatan 9, S-163 91 Spånga
Tel: (46) (0)8 761 70 90, Fax: (46) (0)8 761 81 08
sweden@invacare.com, finland@invacare.com / www.invacare.se

Switzerland:

Invacare AG, Benkenstrasse 260, CH-4108 Witterswil
Tel: (41) (0)61 487 70 80, Fax: (41) (0)61 487 70 81
switzerland@invacare.com / www.invacare.ch

United Kingdom:

Invacare Limited, Pencoed Technology Park, Pencoed, Bridgend CF35 5AQ
Switchboard Tel: (44) (0)1656 776200, Fax: (44) (0)1656 776201
Customer services Tel: (44) (0)1656 776222, Fax: (44) (0)1656 776220
uk@invacare.com / www.invacare.co.uk