VERMEIREN

Freedom - Lagoon - Vintage

INSTRUCTION MANUAL
MODE D'EMPLOI
GEBRUIKSAANWIJZING
GEBRAUCHSANWEISUNG
ISTRUZIONI PER L'USO
MANUAL DE INSTRUCCIONES

Freedom
(2217)

Lagoon
(2219)

Freedom
(2217)

Vintage
(2219)

Instructions to the specialist dealer

This instruction manual is part and parcel of the product and must accompany every product sold.

Version: G, 2015-03

All rights reserved, including translation.

No part of this manual may be reproduced in any form what so ever (print, photocopy, microfilm or any other process) without written permission of the publisher, or processed, duplicated or distributed by using electronic systems.

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© N.V. Vermeiren N.V. 2015

Contents

Preface	2
1 Product description	3
1.1 Intended Use	3
1.2 Technical specifications	4
1.3 Drawing	5
1.4 Location identification plate	5
1.5 Accessories	6
1.6 Explanation of symbols	6
1.7 Safety rules	6
2 Use	7
2.1 Transport the tricycle	7
2.2 Operating the brakes	7
2.3 Operating the gear shifter	8
3 Installation and adjustment	8
3.1 Tools	8
3.2 Manner of delivery	9
3.3 Torque	9
3.4 Adjust the brakes	9
3.5 Adjusting the chains	10
3.6 Adjusting the handlebar	10
3.7 Adjusting the saddle	11
3.8 Tyre changing	11
4 Maintenance	12

Preface

First of all we want to thank you for putting your trust in us by selecting one of our tricycles.

The Vermeiren tricycles are the result of many years of research and experience. During the development, special attention was given to the ease of use and the serviceability of the tricycles.

The expected lifetime of your tricycle is strongly influenced by the care and maintenance of the tricycle.

This manual will help you get acquainted with the operation of your tricycle.

Following the user instructions and the maintenance instructions are an essential part of the warranty.

This manual reflects the latest product developments. Vermeiren has the right to introduce changes without the obligation to adapt or replace previously delivered models.

For any further questions, please consult your specialist dealer.

1 Product description

1.1 *Intended Use*

The tricycle is designed to transport 1 person.

The tricycle is suited for outdoor use.

The user can ride with the tricycle by himself.

The different types of fittings and accessories, and the modular construction allow full use by persons disabled by:

- stiff or damaged joints
- heart insufficiencies and poor blood circulation
- balance disturbances
- cachexia (decrease in muscle)
- and also for aged persons.

When providing for individual requirements:

- body size and weight (max. 110 kg for 2217 and 125 kg for 2219)
- physical and psychological condition
- residential circumstances
- environment

should be taken into consideration.

The tricycle should not be used to transport heavy objects.

Use only Vermeiren approved accessories.

The manufacturer is not liable for damage caused by the lack of or improper service or as a result of not following instructions from this manual.

Compliance with the user and maintenance instructions are an essential part of the guarantee conditions.

Visually impaired people can contact the dealer for the instructions for use.

1.2 Technical specifications


Technical terms below are valid for the tricycle in standard settings.

Commercial terms below are valid for the tricycle in standard settings.


Make	Vermeiren		
Address	Vermeirenplein 1/15, B-2920 Kalmthout		
Type	Tricycle		
Description	2217 Freedom	2219 Lagoon, Vintage	
Maximum occupant mass	110 kg	125 kg	
Maximum load basket	25 kg	25 kg	
Overall width	670 mm	768 mm	
Overall length	1600 mm	1808 mm	
Overall height	986 mm - 1174 mm	1121 mm - 1309 mm	
Total mass	25,5 kg	27,80 kg	
Maximum safe slope	7°	7°	
Ground clearance	97 mm	125 mm	
Height frame tube (entry height)	174 mm	194 mm	
Diameter front tires	20`` x 1,75	24" x 1 3/8"	
Pressure front tires	2,5 - 5 bar	3,5 bar	
Diameter rear tires	20`` x 1,75	24" x 1 3/8"	
Pressure rear tires	2,5 - 5 bar	3,5 bar	
Height saddle	709 mm - 950 mm	789 mm - 1030 mm	
Height handgrips (measured from the ground)	It depends of handlebar model and stem	It depends of handlebar model and stem	
Minimum height pedal	123 mm	150 mm	
Maximum height pedal	373 mm	455 mm	
Distance between saddle and handlebar	372 mm - 401 mm Adjustable: -20 mm	417 mm - 446 mm Adjustable: -20 mm	
Turning diameter	2750 mm	3000 mm	
Rear gear	3	3/7	
Front gear	-	-	
Gear ratio	184%	184% / 184%	
Brakes	V-brake-Front Disc brake - Rear (torpedo brake available -- disc brake as option)	V-brake-Front Disc brake - Rear (torpedo brake available -- disc brake as option)	
Storage and use temperature	+ 5 °C - +41°C		
Storage and use humidity	30% - 70%		
We reserve the right to introduce technical changes. Measurement tolerance ± 15 mm / 1,5 kg / °			

Table 1: Technical specifications

1.3 Drawing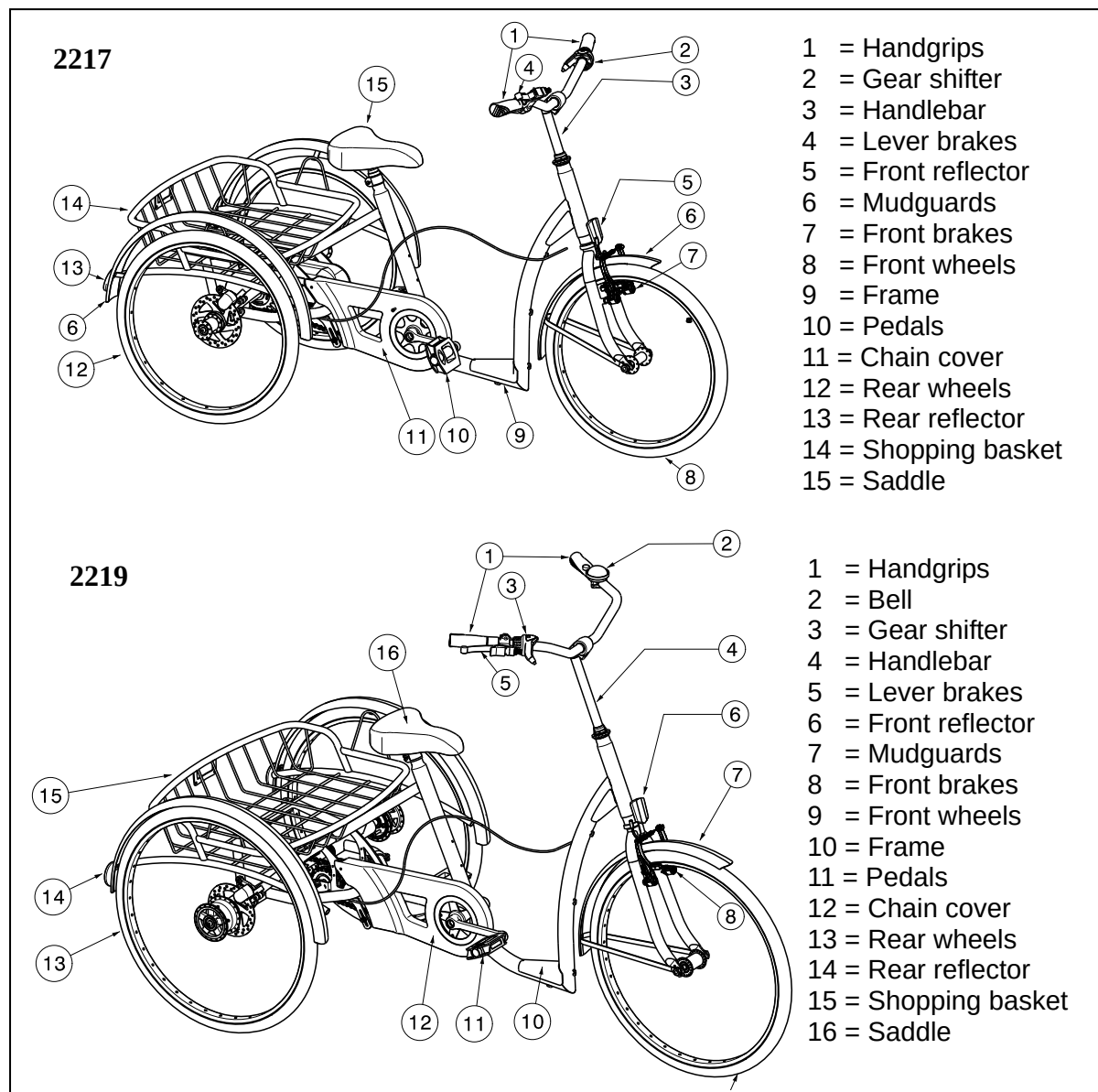

1.4 Location identification plate

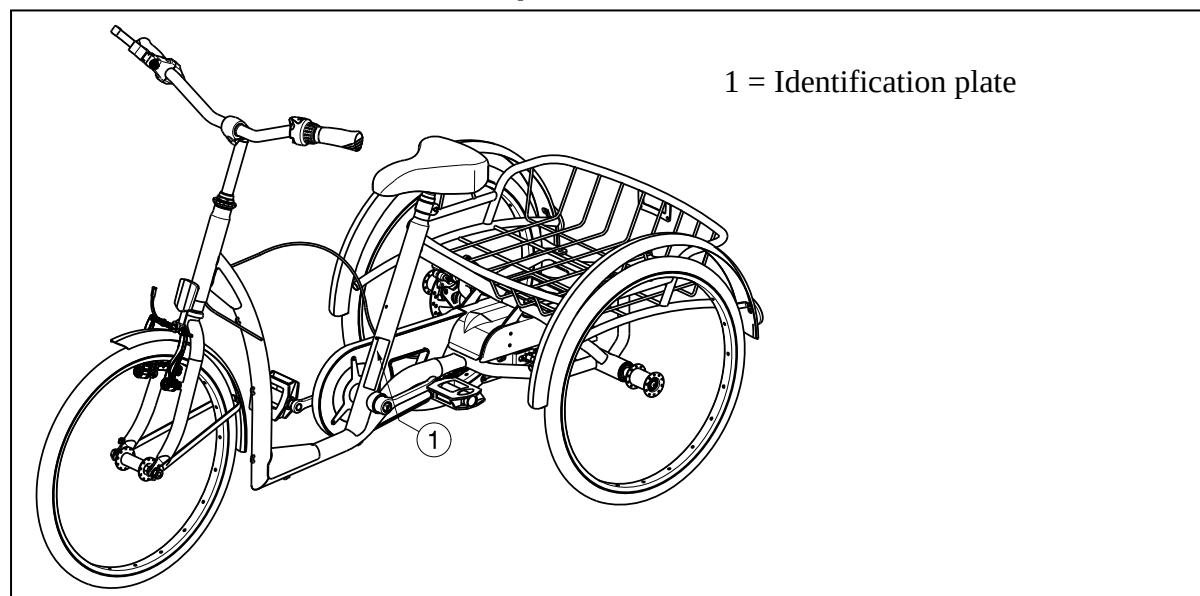1.5 Accessories

- Handlebars
- Adjustable handlebar bracket
- Hand holder / belt
- Handlebar lock system
- Back support / pelvic support
- Adjustable saddles
- Saddle tube with shock absorber
- Pedals with straps / foot supports
- Disc brake on rear wheels
- Lights
- Speedometer
- Vintage basket
- Lockable topcase
- Crutch holder
- Drink holder
- Mirror

Contact your specialist dealer about accessories. He will gladly advise you.

1.6 Explanation of symbols

Maximum mass

CE conformity

Type designation

1.7 Safety rules

- ⚠ To prevent injury and/or damage to your tricycle, make sure that no objects and/or body parts are caught in the spokes of the wheels.
- ⚠ Investigate the effects of shifting the centre of gravity on the behavior of the tricycle, for example on up or down gradients, on laterally sloping ground, or when overcoming obstacles.
- ⚠ The tricycle must be used according the applicable traffic regulations.
- ⚠ When riding your tricycle, you should not be under the influence of alcohol or medicine as in the case of riding other vehicles.
- ⚠ Adapt your riding to weather and traffic conditions.
- ⚠ To be better visible when riding in the dark, wear the brightest possible clothing or clothes with reflectors, and check that the reflectors mounted on the front and rear of the tricycle are clearly visible.
- ⚠ Never exceed the maximum load of your tricycle.
- ⚠ Risk of burns – Be careful when riding in hot or cold environments (sunshine, extreme cold, saunas, etc.) for a sufficient amount of time and when touching - Surfaces can assume the environment temperatures.
- ⚠ Be careful when riding in bends the tricycle can loss sideways stability.

2 Use

This chapter describes the everyday use. **These instructions are for the user and the specialist dealer.**


The tricycle is delivered fully assembled by your specialist dealer. Only the handlebar needs to be adjusted upwards. The instructions intended for the specialist dealer on how to set up the tricycles are given in § 3.

2.1 Transport the tricycle

The best way to move the tricycle is to make use of the wheels and roll the tricycle to the desired place.

2.2 Operating the brakes

- ⚠ **WARNING:** Good operation of the brakes is influenced by wear and contamination of the tires (water, oil, mud, ...) – Check the condition of the tires before each use.
- ⚠ **WARNING:** The brakes are adjustable and can wear – Check the operation of the brakes before each use.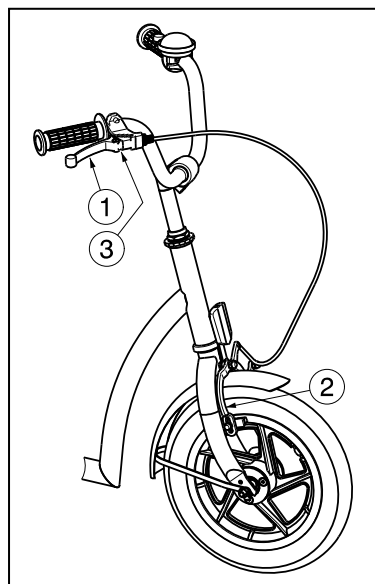

To apply the front wheel brake:

1. Push the brake lever ① to the handgrip.
2. The brake ② shall be pressed against the front wheel.

To release the front wheel brake:

1. Release the brake lever ①.


The rear wheel brake can be activated with the same instructions like the front wheel brake. Left or right side lever can be chosen with the purchase.

Activating the parking brakes:

Block the brakes by pushing the plate ③.

To apply the torpedo brake off the rear wheels:

1. Turn the pedals ① backwards.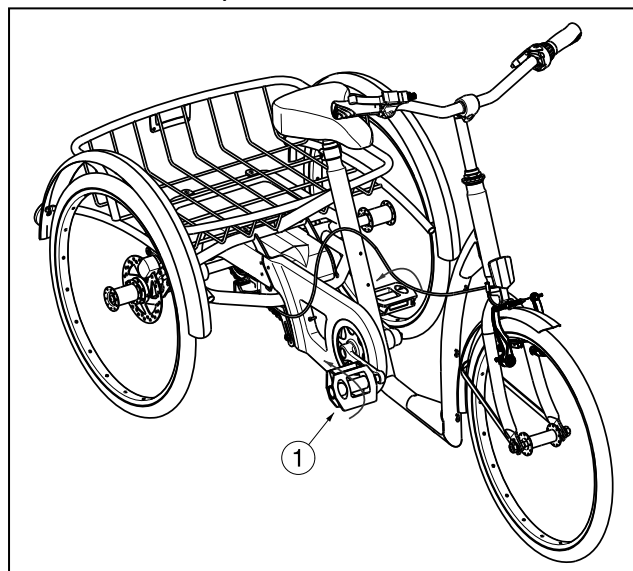


2.3 Operating the gear shifter

The 3 - gear shifter is available for the 2217 and 2219 tricycle.

⚠ CAUTION: Risk of injury - Be sure to shift the lever one gear at a time, and reduce the force being applied to the pedals during shifting. If you try to force operation of the shifting lever while the pedals are being turned strongly, your feet may come off the pedals and the tricycle may topple over.

Turn the shift lever to shift to each of the three gears.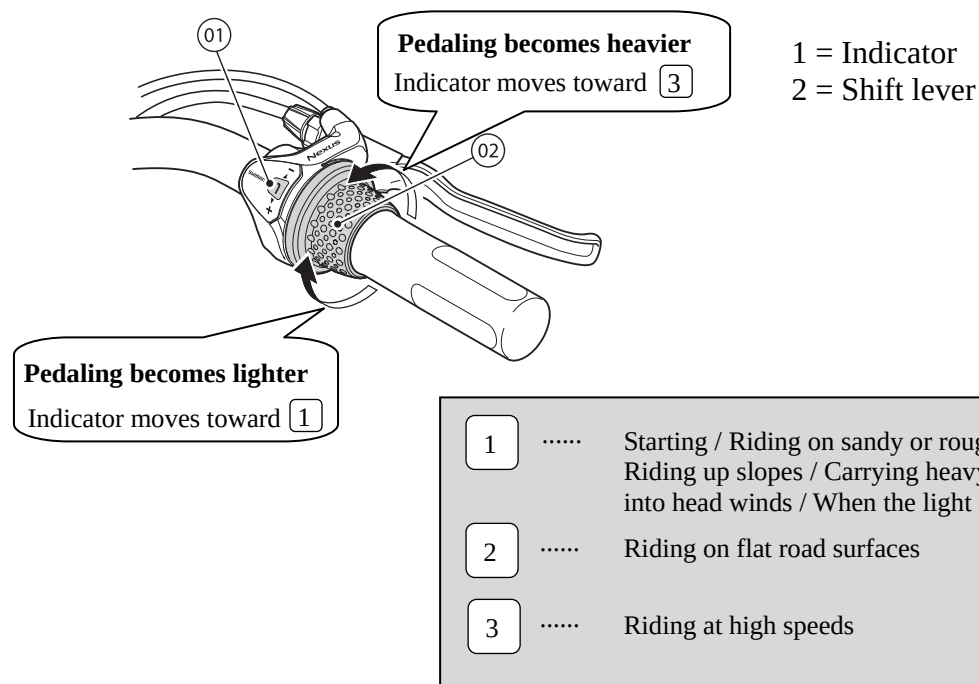

The 7 - gear shifter is only available 2219 tricycle.

3 Installation and adjustment

The instructions in this chapter are for the specialist dealer.

The Vermeiren tricycles 2217, 2219 have been designed entirely for your comfort. This tricycle can be adjusted to individual requirements as explained below.

To find a service facility or specialist dealer near you, contact the nearest Vermeiren facility. A list of Vermeiren facilities can be found on the last page.

⚠ WARNING: Risk of unsafe settings - Use only the settings described in this manual.

⚠ WARNING: Variation of allowed adjustments can still change the stability of your tricycle (tilt back or sideways).

3.1 Tools

To set up the tricycle the following tools are needed.


- Wrench set n° 13
- Allen key set n° 5, 6

3.2 Manner of delivery

The Vermeiren tricycles shall be delivered with:


- 1 frame with rear and front wheels
- pedals
- saddle
- handlebar + handgrips
- bell
- basket
- instruction manual
- accessories

3.3 Torque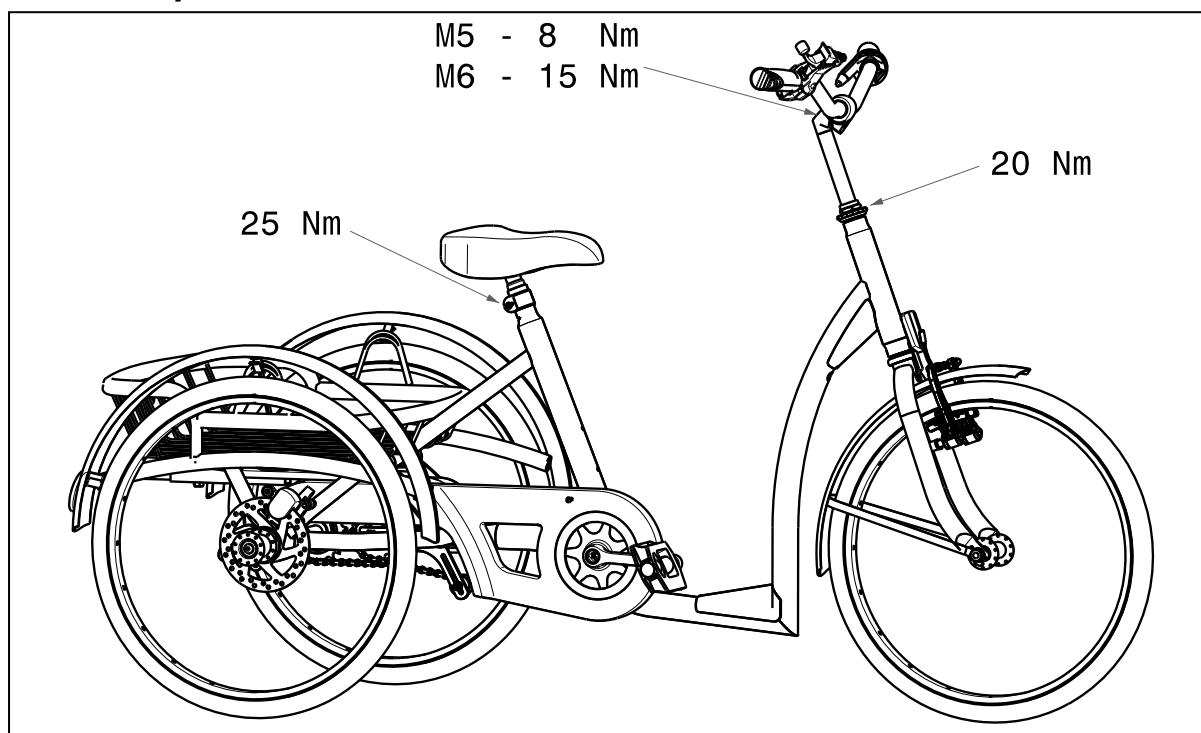


3.4 Adjust the brakes

Adjust the brakes according following rules: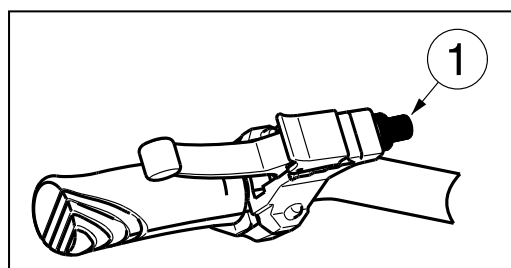

1. The brake tightness can be adjusted by the brake cable adjuster ①.

3.5 Adjusting the chains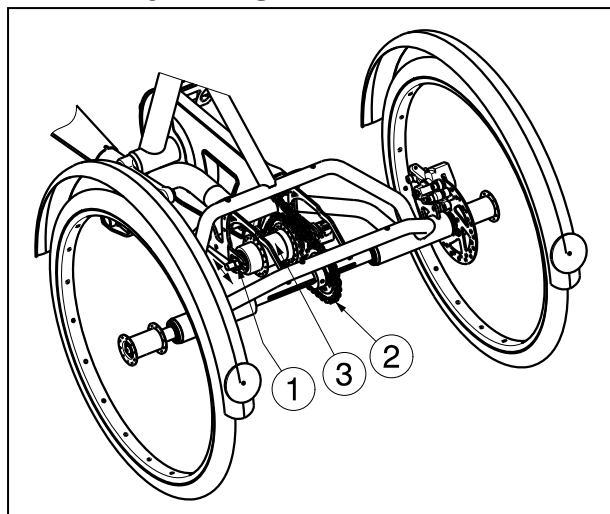

The wheels of the tricycle are driven by a chain ② on the rear axle.


For first time use the chains are set in the factory. After a while it is possible that the chains runs off the axle.

Tightening torpedo chain ②:

Loosen the nut ①. The torpedo system ③ can now be moved on the frame of the tricycle.

When the chain between the two chain-wheels can be moved ± 20 mm, the chain tension is correct.

Retighten the nut ①.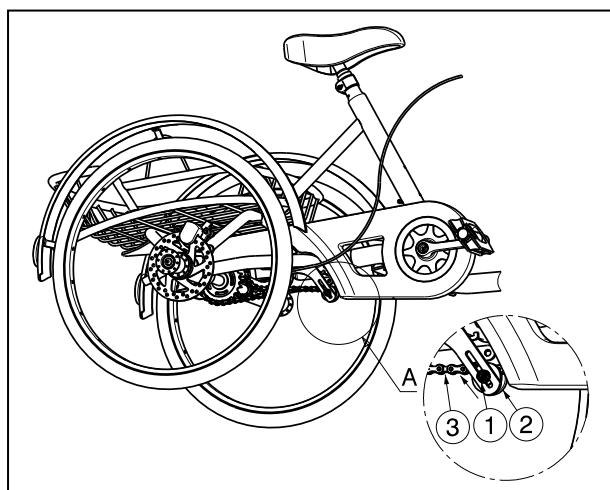

Tightening driving chain ③:


Loosen the nut ①. The wheel ② can now be moved upwards to tighten the chain ③.

When the chain between the two chain-wheels can be moved ± 20 mm, the chain tension is correct.

Retighten the nut ①.

3.6 Adjusting the handlebar

⚠ CAUTION: Risk of injury – Check that the handlebar is firmly secure before using the tricycle.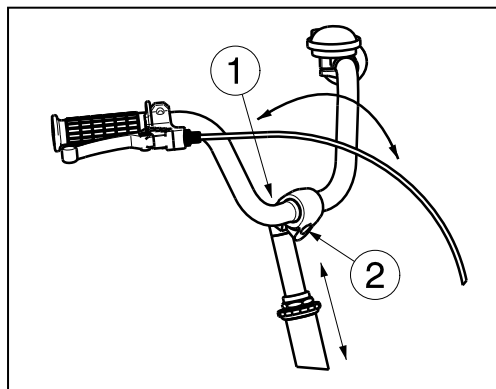

The height and angle of the handlebar can easily be adjusted.

Adjusting height:


1. Loosen the screw ①.
2. A softly tap on the head of the cone.
3. Adjust the height of the handlebar to a comfortable position (stepless -- depends on handlebar model).
4. Tighten the screw ① properly.

Adjusting angle:

1. Loosen the screw ②.
2. Adjust the angle of the handlebar to a comfortable position (stepless).
3. Tighten the screw ② properly.


3.7 Adjusting the saddle

- ⚠ **CAUTION:** Risk of injury - Never adjust the saddle when the user is sitting on the saddle.
- ⚠ **CAUTION:** Risk of injury – Check that the saddle is firmly secure before using the tricycle.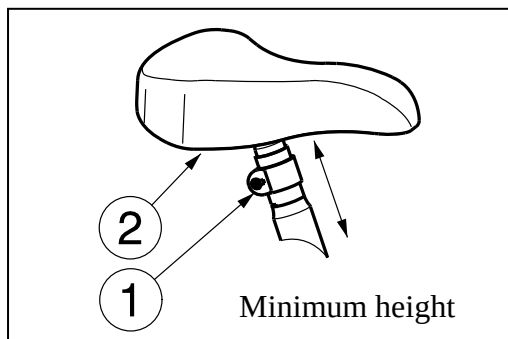

Adjust the height of the saddle as follow (stepless):

1. Loosen the screw ①.
2. Adjust the height of the saddle ② to a comfortable position.
3. Tighten the screw ① properly.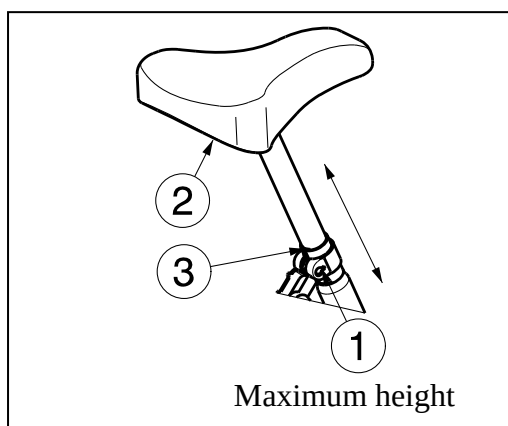

The maximum height is indicated with a straight line ③.

3.8 Tyre changing

- ⚠ **CAUTION:** There must be no air in the tube before it can be removed.
- ⚠ **CAUTION:** Risk of damage - If handled improperly, the rim might be damaged.


If you want to change the tyres or inner tubes, you should note the following:
Before you can remove a tyre, you must let all the air out of the tube, and insert a tyre lever between the tyre and the rim. Then slowly and carefully push the lever downwards. This will pull the tyre over the edge of the rim. If you then move the lever along the rim, the tyre will jump out. Now carefully remove the tyre from the rim and then remove the tube.

Note the following before inserting the new tube:

Check the rim bed and the inside wall of the tyre for foreign matter and clean these properly if necessary. Check the condition of the rim bed, especially around the position of the air valve. Please use only genuine original replacement parts. No liability is accepted for damage caused by non-genuine replacement parts. Kindly contact your specialist dealer.

Assembly:

- ⚠ WARNING: Risk of injury - Check that the pressure is correct.**
- ⚠ CAUTION: Risk of injury - Make sure that no objects or body parts are pinched between the tyre and the rim when mounting a tyre.**

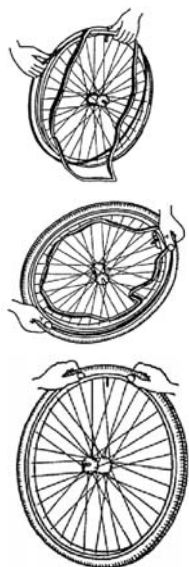Place the rim belt in position over the air valve before inserting it into the rim. Then the rim belt can be pulled over easily. Check that all spoke heads are covered (a rim belt is not required in the case of a plastic rim).

Push the tyre over the edge of the rim, starting behind the air valve. Inflate the tube slightly until it is round, and place it inside the tyre.

If the tube fits snugly inside the tyre without any folds (in the case of folds: let out some air), then the upper side of the tyre can be pressed lightly onto the rim with both hands, starting at the air valve.

Check all around on both sides that the tube is not pinched between the rim and the edge of the tyre. Lightly push the air valve inwards and pull it out again to make sure that the tyre is positioned properly in the region of the air valve.

To ensure that the wheel is inflated correctly, admit only so much air initially that the tyre can still be easily pushed inwards by using your thumbs. If the check-lines are equidistant from the edge of the rim on both sides of the tyre, then the tyre is centered properly. If not - let out the air and position the tyre afresh. Now the tyre can be inflated to its full operating pressure (note the maximum) and the valve cap should be replaced.

Only an expert can guarantee correct assembly. Work not done by your specialist dealer, would void any warranty claims.

When inflating the tyres, always check that the pressure is correct. The correct pressure is given on the tyre walls.

Use only inflating equipment which complies with regulations and indicates the pressure in bar. We do not accept any liability for damage caused by using inflation equipment not supplied by the manufacturer.

4 Maintenance

For the maintenance manual of the tricycles refer to the Vermeiren website:
www.vermeiren.be.

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ENGLISH

CONTRACTUAL WARRANTY

We offer 5 years of warranty on standard wheelchairs, lightweight wheelchairs 4 years. Electronic wheelchairs, tricycles, beds and other products: 2 years (batteries 6 months) and multiposition wheelchairs 3 years. This warranty is limited to the replacement of defective or spare parts.

APPLICATION CONDITIONS

In order to claim its warranty, part "B" of this card has to be given to your official Vermeiren dealer. The warranty is only valid when parts are replaced by Vermeiren in Belgium.

EXCEPTIONS

This warranty is not valid in case of:

- damage due to incorrect usage of the wheelchair,
- damage during transport,
- involvement in an accident,
- a dismount, modification or repair carried outside of our company and/or official Vermeiren dealership,
- normal wear of the wheelchair,
- non-return of the warranty card

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- please return within 8 days of date of purchase or register your product at our website, <http://www.vermeiren.be/registration>

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Vermeiren Austria GmbH
Vermeiren Suisse S.A.