Aquatec® 90000

| | | |
|----|--|----|
| en | Toilet Seat Raiser User Manual | 3 |
| de | Toilettensitzerhöhung Gebrauchsanweisung | 5 |
| fr | Rehausse de siège de toilettes Manuel d'utilisation | 9 |
| it | Rialzo per tavoletta del WC Manuale d'uso | 12 |
| es | Elevador de asiento de inodoro Manual del usuario | 15 |
| nl | Verhoging voor toiletzitting Gebruiksaanwijzing | 18 |
| da | Toiletsædeforhøjer Brugsanvisning | 22 |
| sv | Toalettstshöjare Bruksanvisning | 25 |
| no | Toalettseteheber Bruksanvisning | 28 |
| fi | WC-istuimen koroke Käyttöohje | 31 |

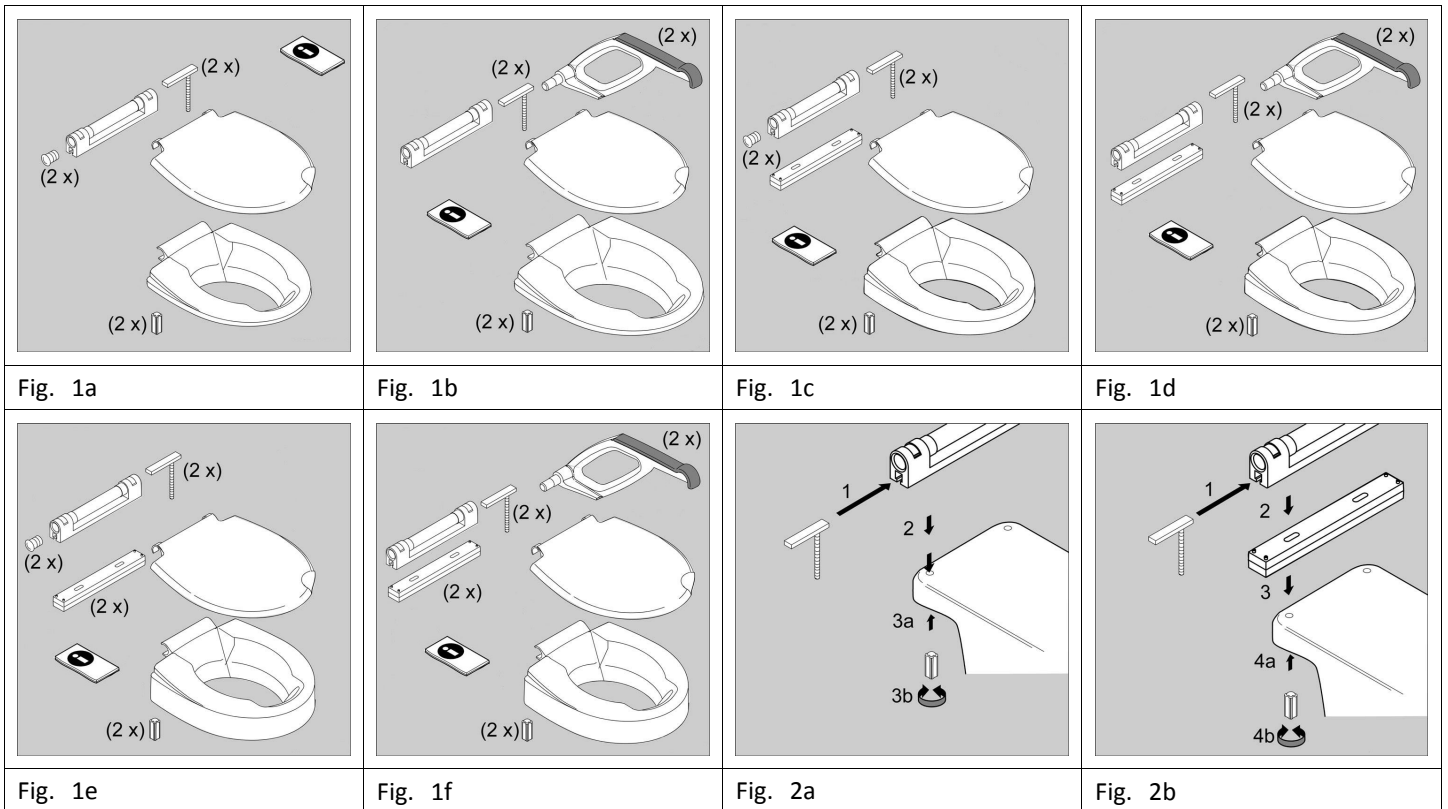
Fig. 1a Fig. 1b Fig. 1c Fig. 1d Fig. 1e Fig. 1f Fig. 2a Fig. 2b

This manual **MUST** be given to the user of the product.
BEFORE using this product, this manual **MUST** be read and saved for future reference.

AQUATEC®

Yes, you can.®

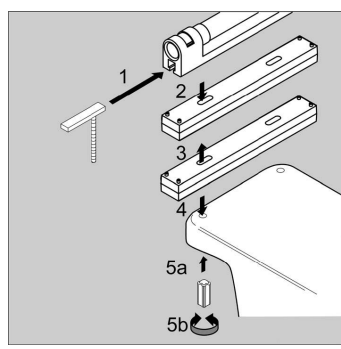
Fig. 2c

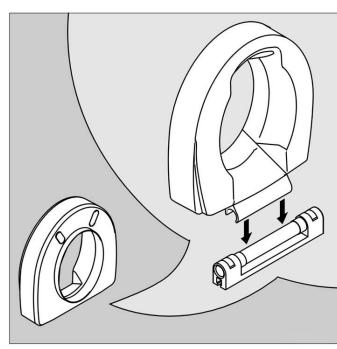
Fig. 3

Fig. 4

Fig. 5

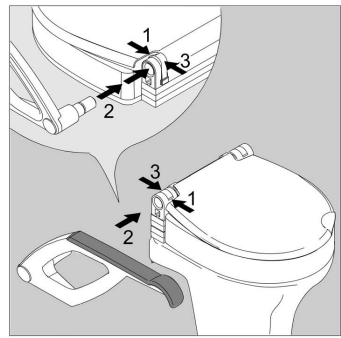
Fig. 6

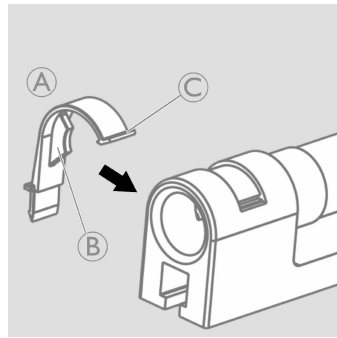
Fig. 7

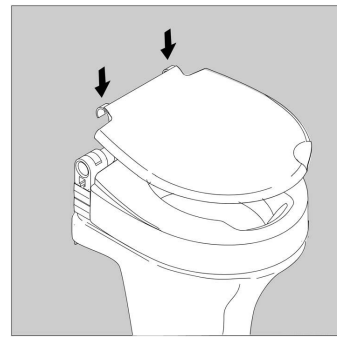
Fig. 8

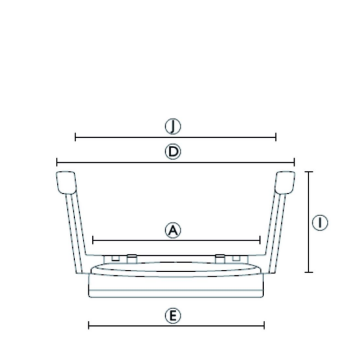
Fig. 9a

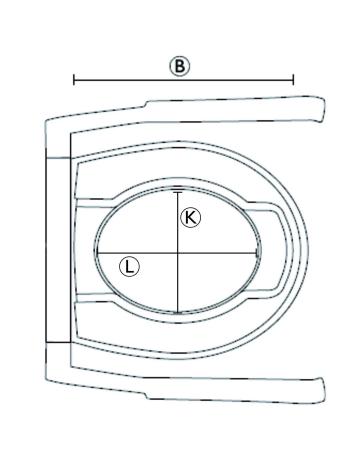
Fig. 9b

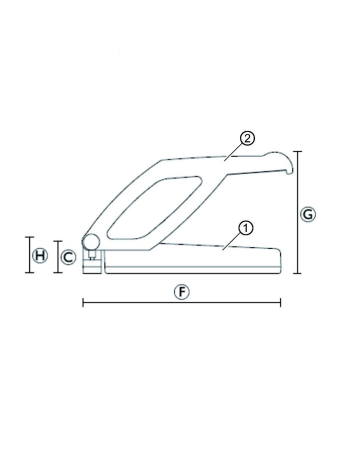
Fig. 9c

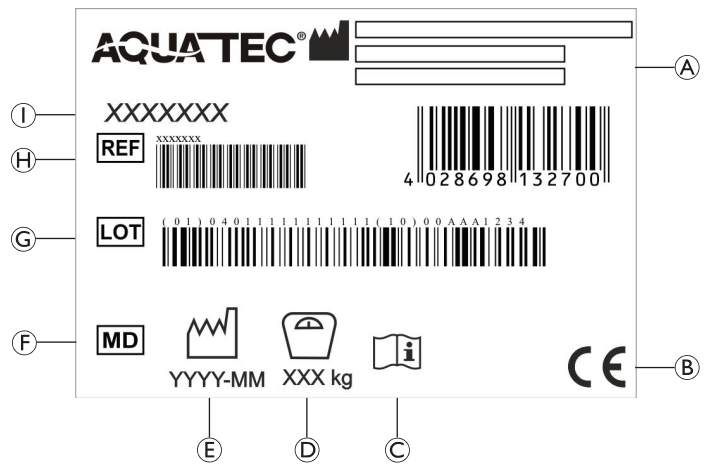
Fig. 10

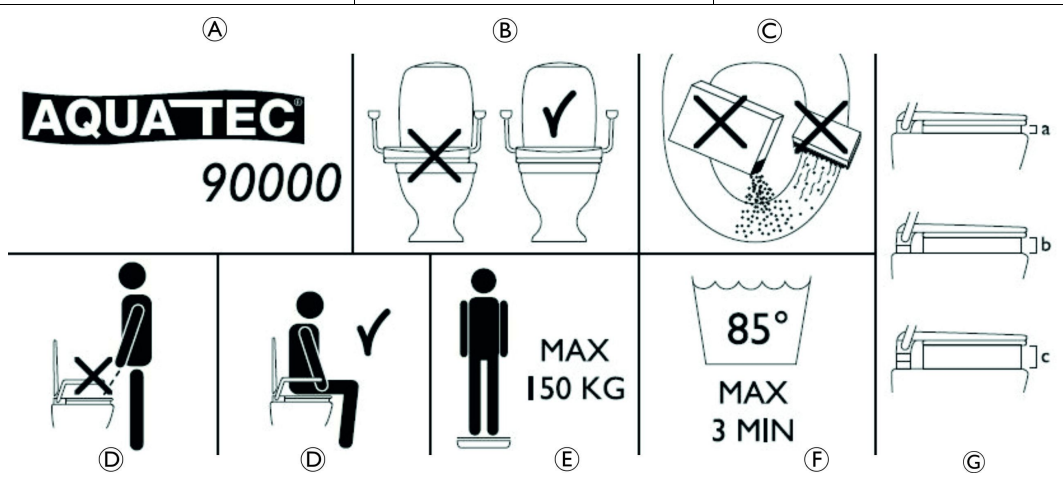
Fig. 11

1 Introduction

This user manual contains important information about the handling of the product. To ensure safety when using the product, read the user manual carefully and follow the safety instructions.

Invacare reserves the right to alter product specifications without further notice.

Before reading this document, make sure you have the latest version. You find the latest version as a PDF on the Invacare website.

If you find that the font size in the printed document is difficult to read, you can download the PDF version from the website. The PDF can then be scaled on screen to a font size that is more comfortable for you.

In case of a serious incident with the product, you should inform the manufacturer and the competent authority in your country.

2 Warranty Information

We provide a manufacturer's warranty for the product in accordance with our General Terms and Conditions of Business in the respective countries.

Warranty claims can only be made through the provider from whom the product was obtained.

3 Compliance

Quality is fundamental to the company's operation, working within the disciplines of ISO 13485.

This product features the CE mark, in compliance with the Medical Device Regulation 2017/745 Class I. The launch date of this product is stated in the CE declaration of conformity.

We are continuously working towards ensuring that the company's impact on the environment, locally and globally, is reduced to a minimum.

We only use REACH compliant materials and components.

3.1 Product-specific Standards

The product has been tested and conforms to EN 12182 (Assistive products for persons with disability) and related standards. For further information about local standards and regulations, contact your local Invacare representative. See addresses at the end of this document.

4 Service Life

The expected service life of this product is three years when used daily and in accordance with the safety instructions, maintenance intervals and correct use, stated in this manual. The effective service life can vary according to frequency and intensity of use.

5 Limitation of Liability

Invacare accepts no liability for damage arising from:

- Non-compliance with the user manual
- Incorrect use
- Natural wear and tear
- Incorrect assembly or set-up by the purchaser or a third party
- Technical modifications
- Unauthorised modifications and/or use of unsuitable spare parts

6 Safety

6.1 Safety Information

WARNING!

Risk of Serious Injury or Damage

Improper use of this product may cause injury or damage.

- If you are unable to understand the warnings, cautions or instructions, contact a health care professional or provider before attempting to use this equipment.
- Do not use this product or any available optional equipment without first completely reading and understanding these instructions and any additional instructional material such as user manual, service manual or instruction sheet supplied with this product or optional equipment.

WARNING!

Risk of Falling

- Do not use if defective.
- Do not make any unauthorized alterations or modifications to the product.
- The armrests are intended to be used on both sides evenly. If pressure is applied on one side only, use only half of the body weight. At overload, there is a risk that the armrest will break.
- Do not use this product as a climbing aid.
- Do not stand on this product.

CAUTION!

Risk of Trapping your Fingers

- When lowering the armrests, ensure that your fingers are away from the hinges to avoid catching them.

WARNING!

- Before first use, clean the product thoroughly.
- Before every use, check the product for damage and ensure that it is securely assembled.

CAUTION!

Risk of Injury

- Non-original or wrong parts may affect the function and safety of the product.
- Only use original parts for the product in use.
 - Due to regional differences, refer to your local Invacare catalog or website for available options or contact your Invacare distributor. See addresses at the end of this document.

6.2 Labels and Symbols on the Product

Identification Label (Fig. 10)

| | | | |
|-----|---------------------|-----|---------------------|
| (A) | Manufacturer | (F) | Medical Device |
| (B) | European Conformity | (G) | Lot number |
| (C) | Read user manual | (H) | Reference number |
| (D) | Maximum user weight | (I) | Product designation |
| (E) | Production date | | |

The identification label is attached to underside of the product

Additional Label (Fig. 11)

| | |
|---|--|
| Ⓐ | Product designation |
| Ⓑ | Product must be positioned centrally on the toilet bowl. (See Chapter 10) |
| Ⓒ | Never use corrosive fluids or abrasive agents for cleaning. (See Chapter 11) |
| Ⓓ | Please sit down on the toilet. |
| Ⓔ | Maximum user weight |
| Ⓕ | Maximum cleaning time / temperature. (See Chapter 11) |
| Ⓖ | Sitting height a, b, c. (Depending on purchased model) |

 The additional label is attached to underside of the lid

7 Scope of Delivery (Fig. 1a – 1f)**2 cm, with and without Armrest (Fig. 1a, 1b)**

- Corpus, 2 cm
- Lid
- Base incl. 2x clips
- Screw (2x)
- Plastic nut (2x)
- Plug (2x) (for version without armrest)
- Armrest (2x), (for version with armrest)
- User manual

6 cm, with and without Armrest (Fig. 1c, 1d)

- Corpus, 6 cm
- Lid
- Base incl. 2x clips
- Screw (2x)
- Plastic nut (2x)
- Height adapter
- Plug (2x) (for version without armrest)
- Armrest (2x), (for version with armrest)
- User manual

10 cm, with and without Armrest (Fig. 1e, 1f)

- Corpus, 10 cm
- Lid
- Base incl. 2x clips
- Screw (2x)
- Plastic nut (2x)
- Height adapter (2x)
- Plug (2x) (for version without armrest)
- Armrest (2x), (for version with armrest)
- User manual

8 Product Description

Aquatec® 90000 is a toilet seat raiser that elevates the seating height on toilets for a better seating position. It is available in three different heights: 2 cm, 6 cm, 10 cm. It can be used with our without armrests providing support for sitting down or standing up. It is fixed with a screws system at the rear of the toilet bowl.

9 Intended Use

This product is intended to be used as an aid for self-hygiene care and toilet purposes by users who have limited function in their legs. The seating height on the toilet is increased by the product. It provides additional support for sitting down or standing up from a toilet seat by foldable armrests and is installed on standard toilet bowls.

Indication

Restricted or limited mobility (e.g.: legs and/or hips) or having balance issues to stand up or walk due to transitional or permanent disabilities.

There are no contraindications known when using the product as intended.

The maximum user weight is 150 kg.

10 Setup (Fig. 2a – 8)**CAUTION!****Risk of Pinching**

– Take care during assembly.

Installing the Toilet Seat Raiser (Fig. 2a – 8)

1. Slide the screws in the designated slot in the base on both sides. Place the base and the height adapter(s) (if applicable) on your toilet as shown on the image. (See table) Tighten the plastic nut.

| | |
|--------------|---------|
| Height 2 cm | Fig. 2a |
| Height 6 cm | Fig. 2b |
| Height 10 cm | Fig. 2c |

2. Place the corpus by holding it upright and fixing it to the base. (Fig. 3)
3. Loosen the plastic nuts, align the toilet seat raiser centrally to your toilet. Close the plastic nuts. (Fig. 4)
4. If your product is without armrests install the plugs on both sides. (Fig. 5)
5. Clip on the lid. (Fig. 8)

To dismount the toilet seat raiser reverse step 2 and 1.

Installing the Armrests (if applicable) (Fig. 6, 7)

1. Open the red clip by releasing the upper end of the clip Ⓒ and then the pin Ⓑ of the red clip Ⓐ. (Fig. 7) Then install the armrests and close the red clip. (Fig. 6)

Ensure that the red clips Ⓐ engage at two points: first the pin Ⓑ and then the upper end of the clip Ⓒ. (Fig. 7)

To disassemble the armrests, first release the upper end of the clip Ⓒ and then the pin Ⓑ.

11 Maintenance**11.1 General Maintenance Information**

This product is maintenance-free, provided that the cleaning and disinfection instructions are adhered to.

11.2 Cleaning and Disinfection**General Safety Information****CAUTION!****Risk of Contamination**

– Take precautions for yourself and use appropriate protective equipment.

IMPORTANT!

- Wrong fluids or methods can harm or damage the product.
- All cleaning agents and disinfectants used must be effective, compatible with one another and must protect the materials they are used to clean.
 - Never use corrosive fluids (alkalines, acid etc.) or abrasive cleaning agents. We recommend an ordinary household cleaning agent such as dishwashing liquid, if not specified otherwise in the cleaning instructions.
 - Never use a solvent (cellulose thinner, acetone etc.) that changes the structure of the plastic or dissolves the attached labels.
 - Always make sure that the product is completely dried before taking it into use again.

For cleaning and disinfection in clinical or long-term care environments, follow your in-house procedures.

Cleaning Intervals

IMPORTANT!

- Regular cleaning and disinfection enhances smooth operation, increases the service life and prevents contamination.
- Clean and disinfect the product
- regularly while it is in use,
 - before and after any service procedure,
 - when it has been in contact with any body fluids,
 - before using it for a new user.

Cleaning Instruction

1. Use water and a mild, non-abrasive cleaner to clean the product.
2. Dry with a cloth.

Cleaning in a Clinical Environment

In addition to cleaning as described above we recommend cleaning the product prior to the use by another patient, after ending of the period of use or on a regular basis (depending on which occurs first) at

- 90 °C for 1 minute
- 85 °C for 3 minutes
- 80 °C for 10 minutes

CAUTION!

Risk of burning Skin

- Allow the product to cool after cleaning.

Disinfection Instructions

Method: Follow the application notes for the used disinfectant and wipe-disinfect all accessible surfaces.

Disinfectant: Ordinary household disinfectant.

Drying: Allow the product to air-dry.

12 Disposal

Be environmentally responsible and recycle this product through your recycling facility at its end of life.

Disassemble the product and its components, so the different materials can be separated and recycled individually.

The disposal and recycling of used products and packaging must comply with the laws and regulations for waste handling in each country. Contact your local waste management company for information.

13 Reconditioning

This product is suitable for reuse. To recondition the product for a new user, carry out the following actions:

- Inspection
- Cleaning and disinfection

For detailed information, see *11 Maintenance, page 4*.

Make sure that the user manual is handed over with the product.

If any damage or malfunction is detected, do not reuse the product.

14 Technical Data (Fig. 9a – c)

| Dimensions | | | |
|------------|------------------|---|--------------------|
| Ⓐ | 365 mm | Ⓔ | 230 / 270 / 310 mm |
| Ⓑ | 425 mm | Ⓕ | 30 / 70 / 110 mm |
| Ⓒ | 20 / 60 / 100 mm | Ⓖ | 200 mm |
| Ⓓ | 540 mm | Ⓙ | 450 mm |
| Ⓔ | 380 mm | Ⓚ | 205 mm |
| Ⓕ | 475 mm | Ⓛ | 275 mm |

| Weights | | with armrests | without armrests |
|---------------------|--------|---------------|---|
| | | Aquatec 90000 | Ⓒ = 20 mm 3.6 kg Ⓒ = 60 mm 4 kg Ⓒ = 100 mm 4.4 kg |
| Maximum user weight | 150 kg | | |

| Materials | | | |
|-----------|----|---|----|
| ① | PP | ② | PP |

All product components are corrosion resistant and do not contain natural rubber latex.

15 Environmental Parameters

| Operating Conditions | |
|---------------------------------|---------------------------|
| Ambient temperature | 10 - 40 °C |
| Relative humidity | 15% - 93%, non-condensing |
| Storage and Shipping Conditions | |
| Ambient temperature | 0 - 40 °C |
| Relative humidity | 30% - 75%, non-condensing |

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1. Ouvrez le clip rouge en dégageant son extrémité supérieure ©, puis la broche ② du clip rouge ①. (Fig. 7) Installez ensuite les accoudoirs et fermez le clip rouge. (Fig. 6)
Invacare Limited
Invacare GmbH