Aquatec® 900

en	Toilet Seat Raiser User Manual	4
de	Toilettensitzerhöhung Gebrauchsanweisung	6
fr	Rehausse de siège de toilettes Manuel d'utilisation	9
it	Rialzo per tavoletta del WC Manuale d'uso	12
es	Elevador de asiento de inodoro Manual del usuario	14
pt	Elevador de assento sanitário Manual de utilização	17
nl	Verhoging voor toiletzitting Gebruiksaanwijzing	19
da	Toiletsædeforhøjer Brugsanvisning	22
no	Toalettsetehever Bruksanvisning	25
fi	WC-istuimen koroke Käyttöohje	27
sv	Toalettstshöjare Bruksanvisning	30
cs	Nástavec záchodového prkénka Návod k obsluze	32
hr	Podizač WC sjedala Upute za uporabu	35
et	WC-poti kõrgendus Kasutusjuhend	38

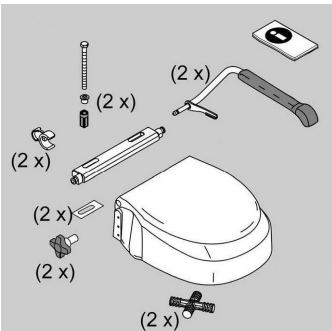
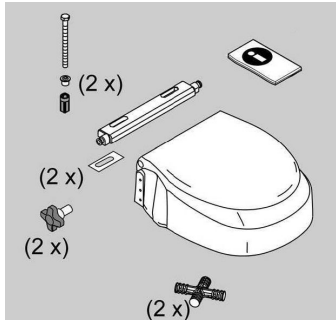
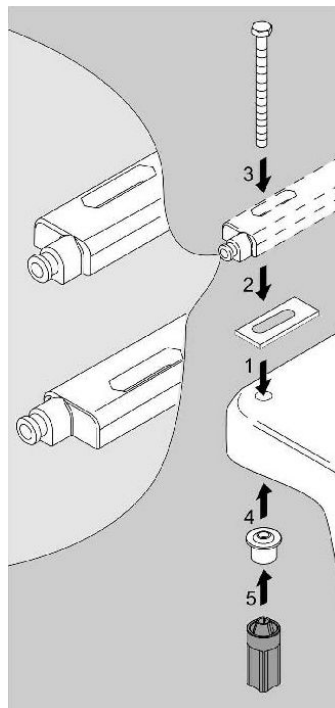
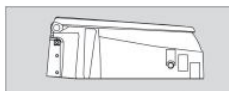
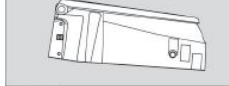
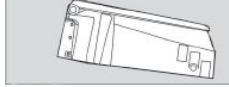
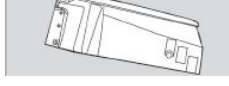
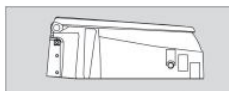
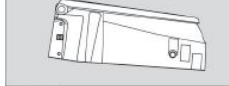
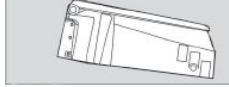
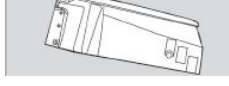
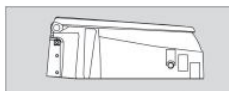
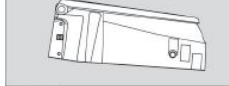
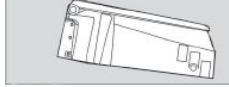
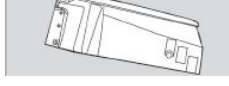
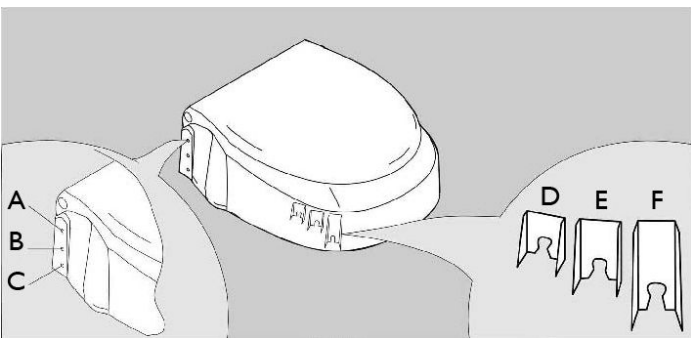
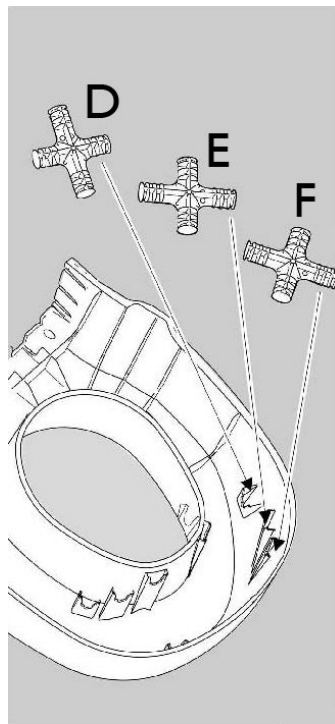
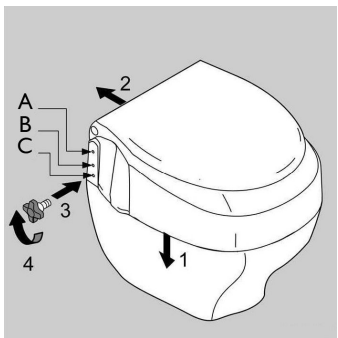
This manual MUST be given to the user of the product.
BEFORE using this product, this manual MUST be read and saved for future reference.

AQUATEC®

Yes, you can.®

			<table><tr><td></td><td>I 6 cm A + D</td></tr><tr><td></td><td>I 10 cm B + E</td></tr><tr><td></td><td>I 15 cm C + F</td></tr><tr><td></td><td>8° B + D</td></tr><tr><td></td><td>11° C + E</td></tr><tr><td></td><td>18° C + D</td></tr></table>		I 6 cm A + D		I 10 cm B + E		I 15 cm C + F		8° B + D		11° C + E		18° C + D
	I 6 cm A + D														
	I 10 cm B + E														
	I 15 cm C + F														
	8° B + D														
	11° C + E														
	18° C + D														
Fig. 1	Fig. 2	Fig. 3	Fig. 4												
															
Fig. 5	Fig. 6	Fig. 7													

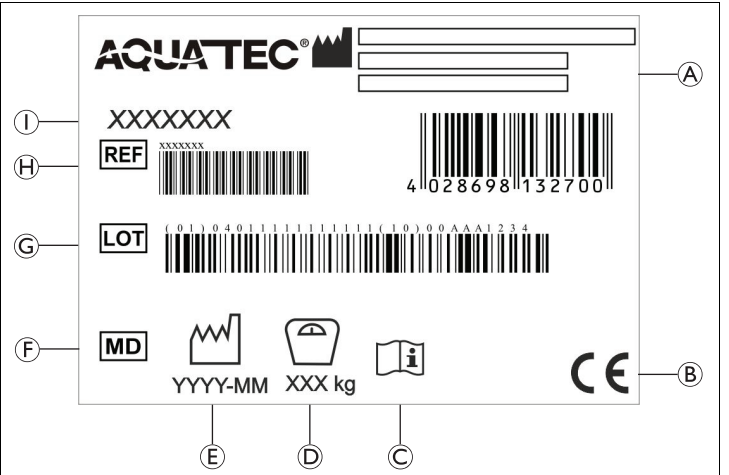
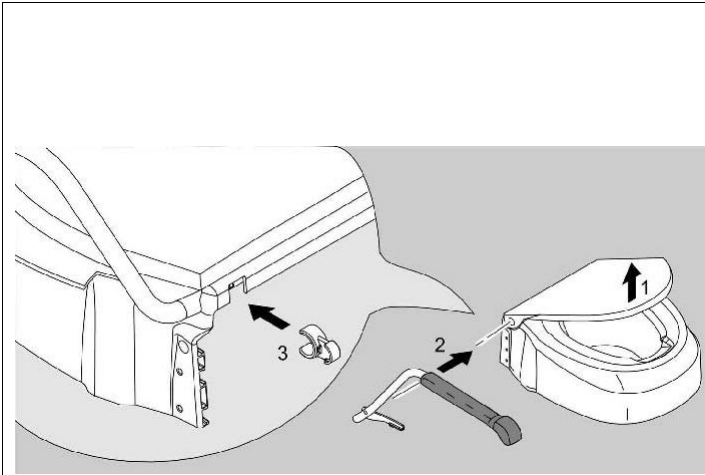
Fig. 8

Fig. 9

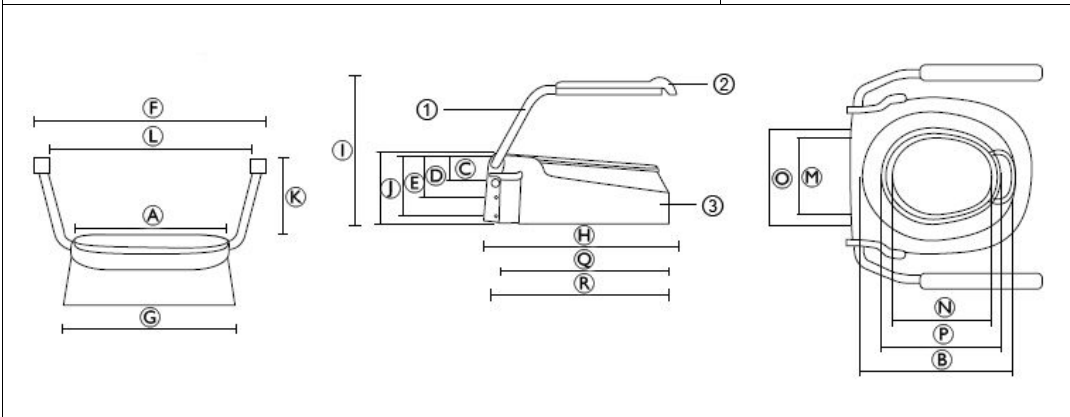
Fig. 10

1 Introduction

This user manual contains important information about the handling of the product. To ensure safety when using the product, read the user manual carefully and follow the safety instructions.

Invacare reserves the right to alter product specifications without further notice.

Before reading this document, make sure you have the latest version. You find the latest version as a PDF on the Invacare website.

If you find that the font size in the printed document is difficult to read, you can download the PDF version from the website. The PDF can then be scaled on screen to a font size that is more comfortable for you.

In case of a serious incident with the product, you should inform the manufacturer and the competent authority in your country.

2 Warranty Information

We provide a manufacturer's warranty for the product in accordance with our General Terms and Conditions of Business in the respective countries.

Warranty claims can only be made through the provider from whom the product was obtained.

3 Compliance

Quality is fundamental to the company's operation, working within the disciplines of ISO 13485.

This product features the CE mark, in compliance with the Medical Device Regulation 2017/745 Class I. The launch date of this product is stated in the CE declaration of conformity.

We are continuously working towards ensuring that the company's impact on the environment, locally and globally, is reduced to a minimum.

We only use REACH compliant materials and components.

3.1 Product-specific Standards

The product has been tested and conforms to the standard ISO 17966 (Assistive products for personal hygiene that support users) and related standards.

For further information about local standards and regulations, contact your local Invacare representative. See addresses at the end of this document.

4 Service Life

The expected service life of this product is three years when used daily and in accordance with the safety instructions, maintenance intervals and correct use, stated in this manual. The effective service life can vary according to frequency and intensity of use.

5 Limitation of Liability

Invacare accepts no liability for damage arising from:

- Non-compliance with the user manual
- Incorrect use
- Natural wear and tear
- Incorrect assembly or set-up by the purchaser or a third party
- Technical modifications
- Unauthorised modifications and/or use of unsuitable spare parts

6 Safety

6.1 Safety Information

WARNING!

Risk of Serious Injury or Damage

Improper use of this product may cause injury or damage.

- If you are unable to understand the warnings, cautions or instructions, contact a health care professional or provider before attempting to use this equipment.
- Do not use this product or any available optional equipment without first completely reading and understanding these instructions and any additional instructional material such as user manual, service manual or instruction sheet supplied with this product or optional equipment.

WARNING!

Risk of Falling

- Do not use if defective.
- Do not make any unauthorised alterations or modifications to the product.
- The armrests are intended to be used on both sides evenly. If pressure is applied on one side only, use only half of the body weight. At overload, there is a risk that the armrest will break.
- Do not use this product as a climbing aid.
- Do not stand on this product.

CAUTION!

Risk of Trapping your Fingers

- When lowering the armrests, ensure that your fingers are away from the hinges to avoid catching them.

WARNING!

- Before first use, clean the product thoroughly.
- Before every use, check the product for damage and ensure that it is securely assembled.

CAUTION!

Risk of Injury

Non-original or wrong parts may affect the function and safety of the product.

- Only use original parts for the product in use.
- Due to regional differences, refer to your local Invacare catalogue or website for available options or contact your Invacare distributor. See addresses at the end of this document.

6.2 Labels and Symbols on the Product

Identification Label (Fig. 9)

(A)	Manufacturer	(F)	Medical Device
(B)	European Conformity	(G)	Lot number
(C)	Read user manual	(H)	Reference number
(D)	Maximum user weight	(I)	Product designation
(E)	Production date		

The identification label is attached to the backside of the corpus.

7 Scope of Delivery (Fig. 1, 2)

- Corpus with lid
- Attachment cross (2x)
- Star screw (2x)
- Crossbar
- Anti-slip pad (2x)
- Screw with nut and washer (2x)
- Clip (2x)
- Armrest (2x) (if applicable)
- User manual

8 Product Description

Aquatec® 900 is intended to be used as an aid for self-hygiene care and toilet purposes by users who have limited function in their legs. The seating height on the toilet is increased by the product. It provides additional support for sitting down or standing up from a toilet seat by foldable armrests and is installed on standard toilet bowls.

9 Intended Use

This product is intended to be used as an aid for self-hygiene care and toilet purposes by users who have limited function in their legs.

Indication

Restricted or limited mobility (e.g.: legs and/or hips) or having balance issues to stand up or walk due to transitional or permanent disabilities.

There are no contraindications known when using the product as intended.

The maximum user weight is 120 kg.

10 Setup (Fig. 3 – 8)

CAUTION!

Risk of Pinching

- Take care during assembly.

1. Remove the old toilet seat.
2. Fix the crossbar with anti-slip protection, screws, nut and washer onto the toilet bowl. (Fig. 3)
3. Choose the desired seat height and inclination from the table in Fig. 4. The letters in Fig. 4 correlate with the possible fixation positions in Fig. 5.
4. Place both attachment crosses in the desired fixation. (Fig. 6)
5. Place the corpus on your toilet bowl, move it backward and hinge the desired slot with the pin of the crossbar. Secure the position with the two star screws. (Fig. 7)
6. Install the armrests and fixate them with the clips. (Fig. 8) (If applicable)
7. If the toilet seat raiser should not fit satisfactory install the crossbar reverted. (Fig. 3)

To dismount the toilet seat raiser reverse steps 6 to 2.

11 Maintenance

11.1 General Maintenance Information

This product is maintenance-free, provided that the cleaning and disinfection instructions are adhered to.

11.2 Cleaning and Disinfection

General Safety Information

CAUTION!

Risk of Contamination

- Take precautions for yourself and use appropriate protective equipment.

IMPORTANT!

Wrong fluids or methods can harm or damage the product.

- All cleaning agents and disinfectants used must be effective, compatible with one another and must protect the materials they are used to clean.
- Never use corrosive fluids (alkalines, acid etc.) or abrasive cleaning agents. We recommend an ordinary household cleaning agent such as dishwashing liquid, if not specified otherwise in the cleaning instructions.
- Never use a solvent (cellulose thinner, acetone etc.) that changes the structure of the plastic or dissolves the attached labels.
- Always make sure that the product is completely dried before taking it into use again.

For cleaning and disinfection in clinical or long-term care environments, follow your in-house procedures.

Cleaning Intervals

IMPORTANT!

Regular cleaning and disinfection enhances smooth operation, increases the service life and prevents contamination.

Clean and disinfect the product

- regularly while it is in use,
- before and after any service procedure,
- when it has been in contact with any body fluids,
- before using it for a new user.

Cleaning Instructions

IMPORTANT!

- Clean by hand
- Maximum cleaning temperature 60 °C

1. Clean the product by wiping down all generally accessible surfaces.
2. Dry the parts with a cloth.

Disinfection Instructions

Method: Follow the application notes for the used disinfectant and wipe-disinfect all accessible surfaces.

Disinfectant: Ordinary household disinfectant.

Drying: Allow the product to air-dry.

12 Disposal

Be environmentally responsible and recycle this product through your recycling facility at its end of life.

Disassemble the product and its components, so the different materials can be separated and recycled individually.

The disposal and recycling of used products and packaging must comply with the laws and regulations for waste handling in each country. Contact your local waste management company for information.

13 Reconditioning

This product is suitable for reuse. To recondition the product for a new user, carry out the following actions:

- Inspection
- Cleaning and disinfection

For detailed information, see *11 Maintenance, page 5*.

Make sure that the user manual is handed over with the product.

If any damage or malfunction is detected, do not reuse the product.

14 Technical Data (Fig. 10)

Dimensions			
Ⓐ	360 mm	Ⓙ	180 mm
Ⓑ	425 mm	Ⓚ	200 mm
Ⓒ	60 mm	Ⓛ	510 mm
Ⓓ	100 mm	Ⓜ	205 mm
Ⓔ	150 mm	Ⓝ	260 mm
Ⓕ	590 mm	Ⓞ	225 mm
Ⓖ	430 mm	Ⓟ	285 mm
Ⓗ	490 mm	Ⓠ	420 mm
Ⓢ	380 mm	Ⓡ	440 mm
Possible distance between the bolt holes of the toilet bowl:		138 mm – 224 mm	

Weights		
Product weight	With armrests	5.0 kg
	Without armrests	2.7 kg
Maximum user weight	120 kg	

Materials			
①	Steel	③	PP
②	PP		

All product components are corrosion resistant and do not contain natural rubber latex.

15 Environmental Parameters

Operating Conditions	
Ambient temperature	10 - 40 °C
Relative humidity	15% - 93%, non-condensing
Storage and Shipping Conditions	
Ambient temperature	0 - 40 °C
Relative humidity	30% - 75%, non-condensing

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Invacare Limited
Invacare GmbH