

INGLESE - cod22824 - 09/11/2020



ORMESA®
• MADE IN ITALY SINCE 1980 •

DONDOLINO



DONDOLINO

Stability in motion.

Dondolino is a prone standing frame designed and built by Ormesa to help children acquire **independence in maintaining a standing posture**.

Dondolino's use in everyday contexts **encourages environmental interaction** by facilitating upper limb and head mobility and the development of individual motor skills.

Dondolino is an extremely versatile tool that comes in three sizes with a wide range of adjustments. It can be used by children and teens from 75 to 150 cm tall, **adapting to their growth, development of individual skills and hip and knee movements**.



PRODUCT REGISTERED 

PRODUCT MARKED 





Tilting frame

Through spring stopper adjuster it is possible to switch from the upright position to a more prone one in order to stimulate the active control of the head and trunk.

These changes of attitude can be applied in a completely fluid way even with the child on board, without causing pathological reactions of postural discomfort.

Dondolino contains without forcing, adapting the structure to different postural needs.



Technical features and adjustments



Frame

for children needing to improve straightening reactions. Protrusion free to guarantee safety. Allows trunk rotation and maximum upper limb freedom of movement. Smooth continuous adjustment (through spring stopper adjuster).



Tools free

adjustment without tools but with special knobs, designed by Ormesa, with effective and safe grip. The knobs effectively maintain tightening thanks to an internal mechanism that does not mark and ruin the tube.




Graduated bars




to help finding the right postural setting chosen by the therapist.



Pelvic support and thoracic support




independently adjustable in circumference, height and forward-backward. If the user has sufficient trunk control, the thoracic support can be removed.

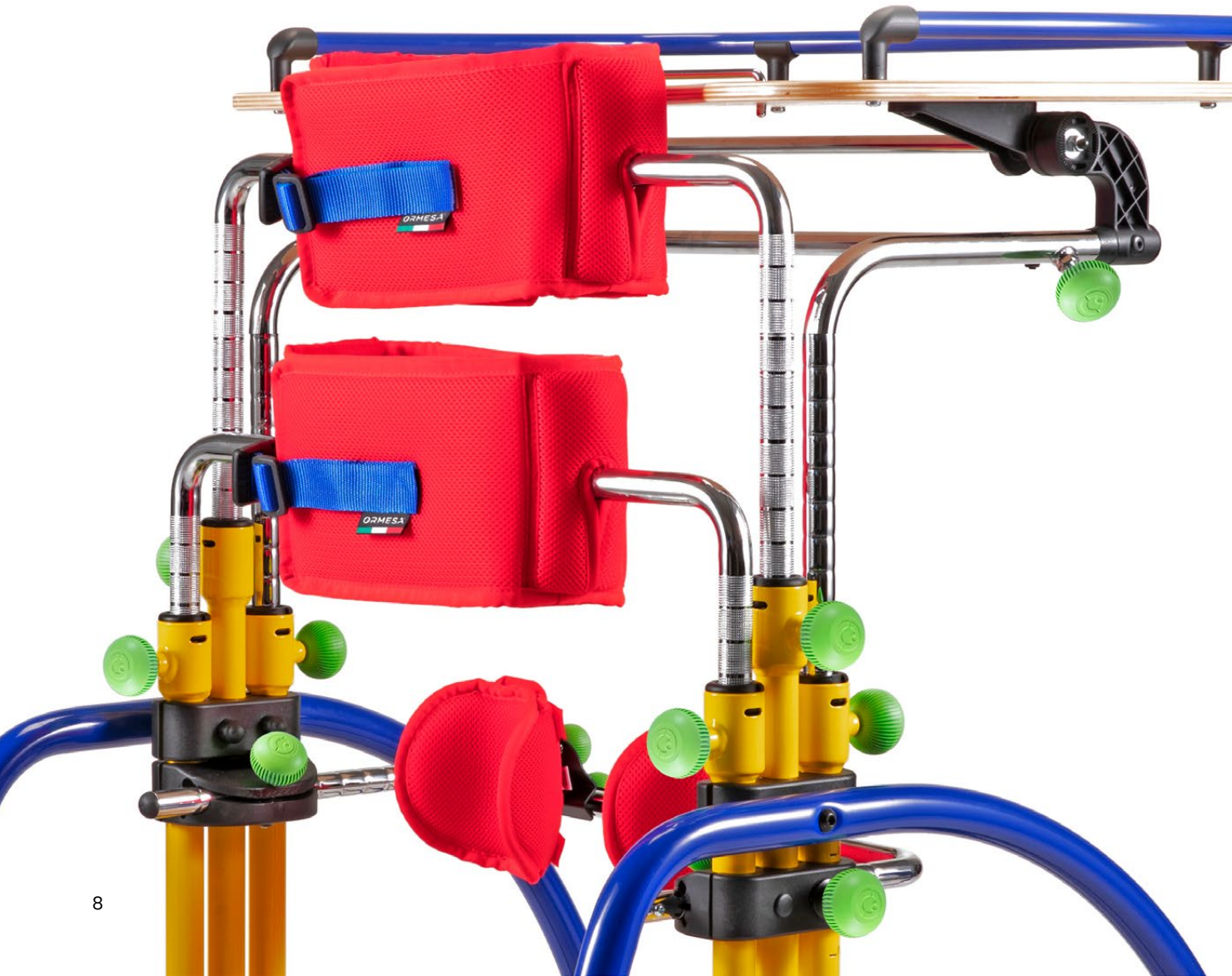

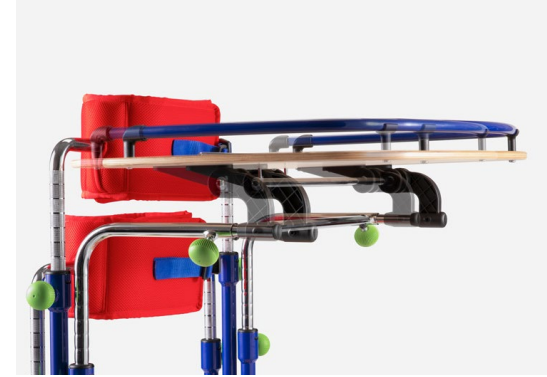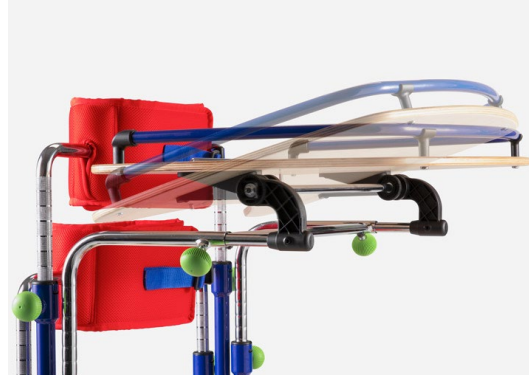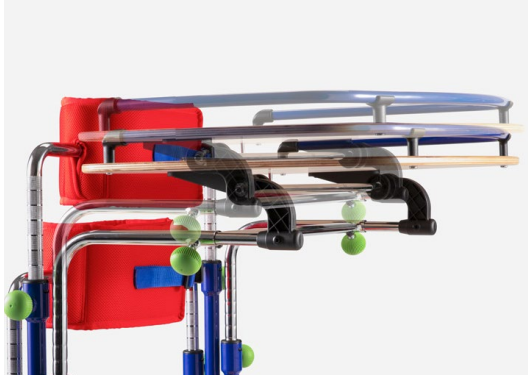


Birch plywood table

designed for educational purposes and for supporting upper limbs, can be removed to facilitate upper limb and trunk exercises. Height, inclination and forward-backward adjustment.

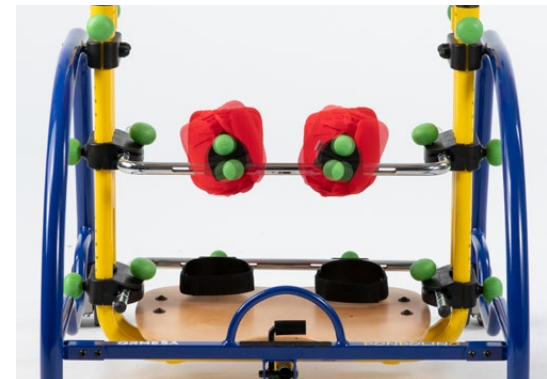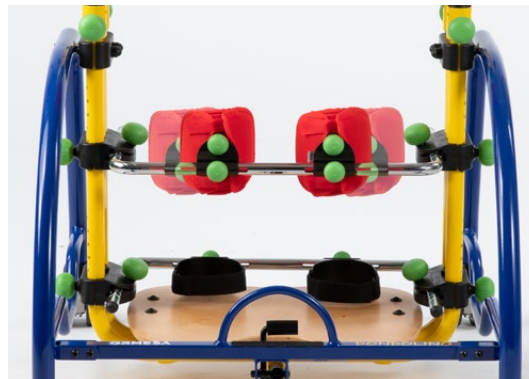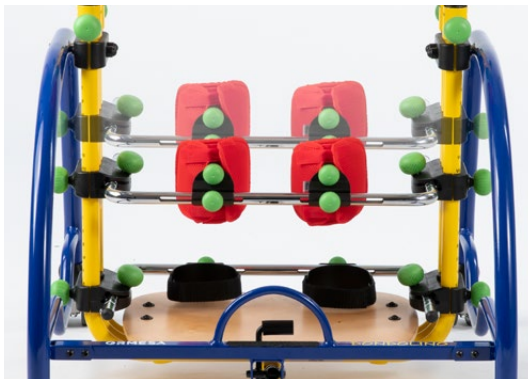



Knee pads

with removable, washable padding and with holes for patella discharge. Independent height, forward-backward, rotation and abductive adjustments.

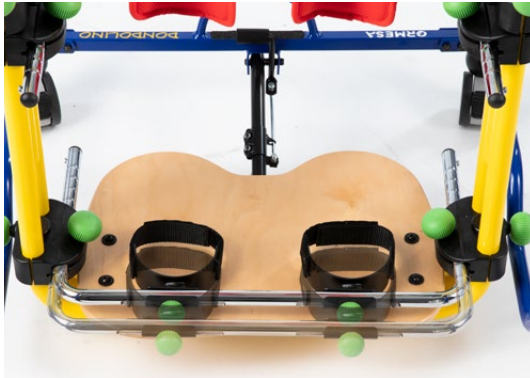

Heel stops

width and depth adjustments.

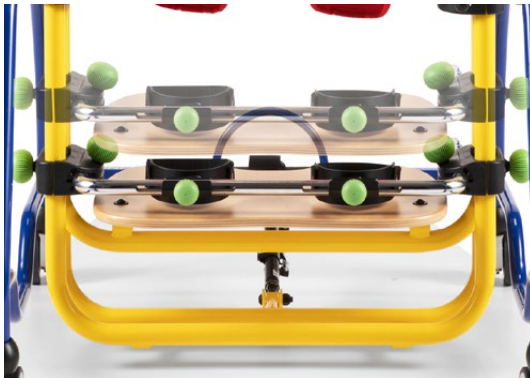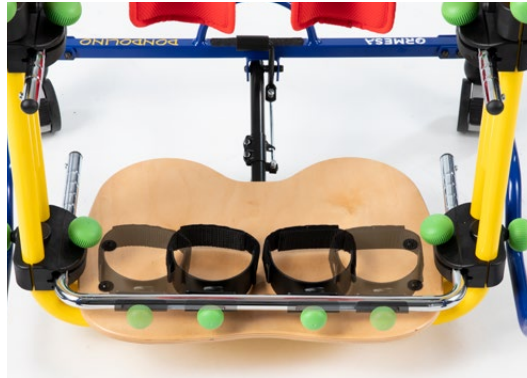

Birch plywood footrest

height adjustments.




Twin swivel wheels


75 mm diameter, all with brakes. The wheels allow movement from one room to another, even during use.



other components

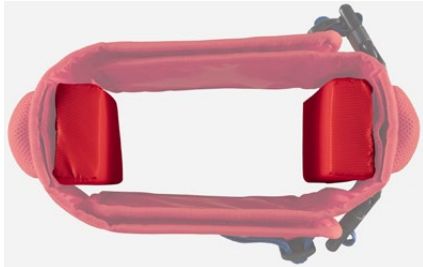

944 Circumference reducers
each 3 cm thick, can be inserted into
supports for thinner users.

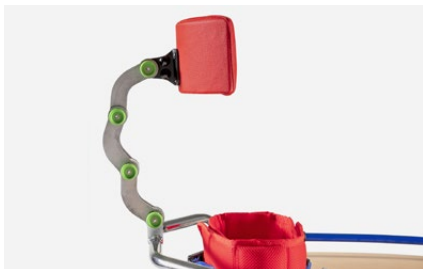


865 Multiadjustable headrest
with height, inclination and forward-
backward adjustment knobs.





Dondolino sizes and weight


| | size 1 | size 2 | size 3 |
|--|-------------------|-------------------|--------------------|
| A: base encumbrance | 63 cm | 77 cm | 83 cm |
| B: length encumbrance | from 75 to 85 cm | from 81 to 91 cm | from 85 to 95 cm |
| C: footrest to table height | from 45 to 79 cm | from 52 to 90 cm | from 62 to 107 cm |
| D: width encumbrance | 63 cm | 67 cm | 71 cm |
| E: footrest to knee pads height | from 12 to 26 cm | from 12 to 36 cm | from 14 to 48 cm |
| F: footrest to pelvic support height | from 39 to 59 cm | from 47 to 74 cm | from 58 to 93 cm |
| G: footrest to thoracic support height | from 51 to 76 cm | from 56 to 90 cm | from 70 to 110 cm |
| L: circumference of supports | 60 cm | 65 cm | 70 cm |
| M: user height | from 75 to 100 cm | from 90 to 125 cm | from 120 to 150 cm |
| P: frame inclination | from 0° to 19° | from 0° to 14° | from 0° to 11° |
| Total weight | 21 kg | 24 kg | 26 kg |
| Maximum load | 35 kg | 45 kg | 55 kg |

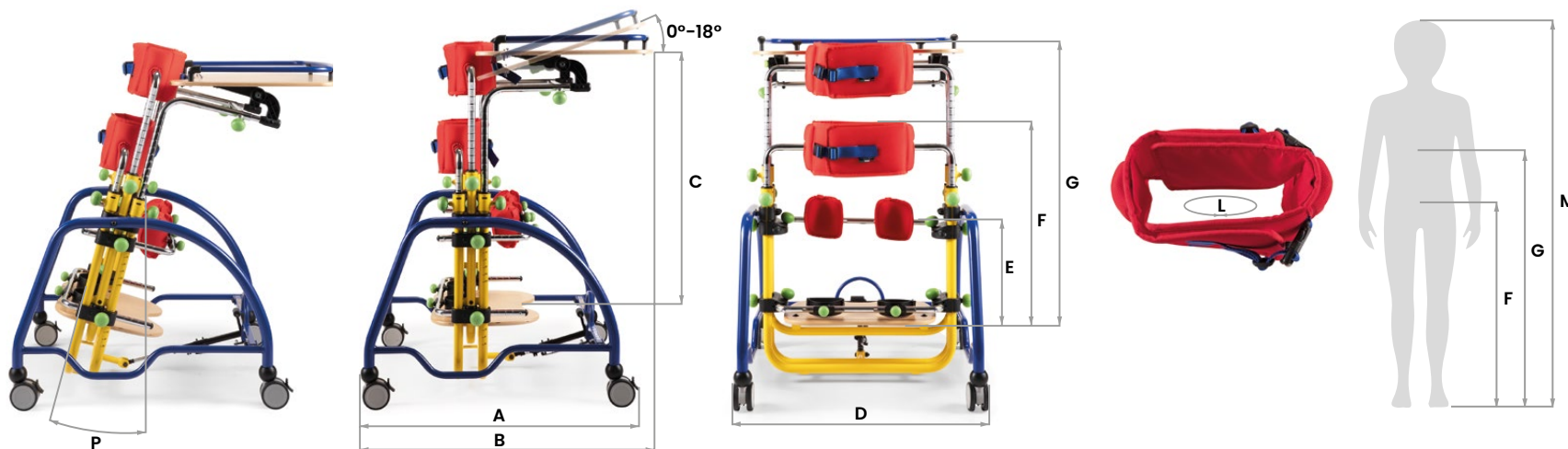

Make sure that measures F and G indicated in the figure do not exceed the suggested range indicated for the supports.

Pietro and his story await you in official website Ormesa, together with other protagonists of our fantastic global community.



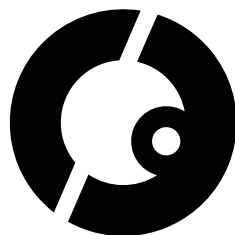

A special thanks to the families who participated with enthusiasm in our project. It is thanks to our customers and their stories that we find the energy and stimuli to evolve day by day.

COMPANY WITH
QUALITY SYSTEM
CERTIFIED BY DNV GL
= **ISO 13485** =

Ormesa srl - Via A. Da Sangallo, 1 | 06034 Foligno (PG) - ITALY
Tel. +39 0742 22927 | FAX +39 0742 22637 | info@ormesa.com

www.ormesa.com