

Spare Parts Catalogue

Version: 2018/05/18a EN



Invacare® **Birdie**^{EVO}



Yes, you can.®

General Information

For any order of spare parts the following information is required:

- Product type*
- Product name & model*
- Serial Number*
- Part-Number*
- Part-Description*

Exploded view drawings and code numbers are not contractual and subject to modification, in relation to Invacare® continuous product improvement policy to meet end users needs.

The reproduction or use of this work, or of any substantial part of this work, in any form or by any electronic, mechanical or other means, including photocopying, scanning and recording, and in any information storage and retrieval system, including electronic transmission, is forbidden without the prior written permission of Invacare® International Sarl.



Yes, you can.®

TABLE OF CONTENTS

Base with manual leg spread, Birdie EVO COMPACT

Base for Foldable Mast	5
Base for Fixed Mast	6
Leg assembly	7
Pedal mechanism assembly	8

Base with manual leg spread, Birdie EVO

Base for Foldable Mast	9
Base for Fixed Mast	10
Leg assembly	11
Pedal mechanism assembly	12

Base with electrical leg spread, Birdie EVO

Base for Foldable Mast	13
Leg assembly	14

Base with electrical leg spread, Birdie EVO PLUS

Base for Foldable Mast	15
Base for Fixed Mast	16
Leg assembly	17

Castors

Birdie EVO COMPACT and Birdie EVO castors	18
Birdie EVO PLUS castors	19

Electrical Parts

Actuator for Boom	20
Actuator for elec. leg spread, Birdie EVO	21
Actuator for elec. leg spread, Birdie EVO PLUS	22
Control units	23
Hand controls	24
Electrical Accessories & Hardware	25

Mast and Boom

Birdie EVO COMPACT, Mast and Boom, Foldable or Fixed	26
Birdie EVO and EVO PLUS, Mast and Boom, Foldable or Fixed	27
Labelling, Birdie EVO	28
Spreader Bar Damper Slow'R	29

Spreader Bars

Spreader bar Hardware	30
Spreader bar 2 points (350/450/550 mm)	31
Spreader bar 4 points (450/550 mm)	32

Accessories


Scale and Attachment	33
Lever	34



Yes, you can.[®]

TABLE OF CONTENTS


Leg Protection	35
----------------------	----

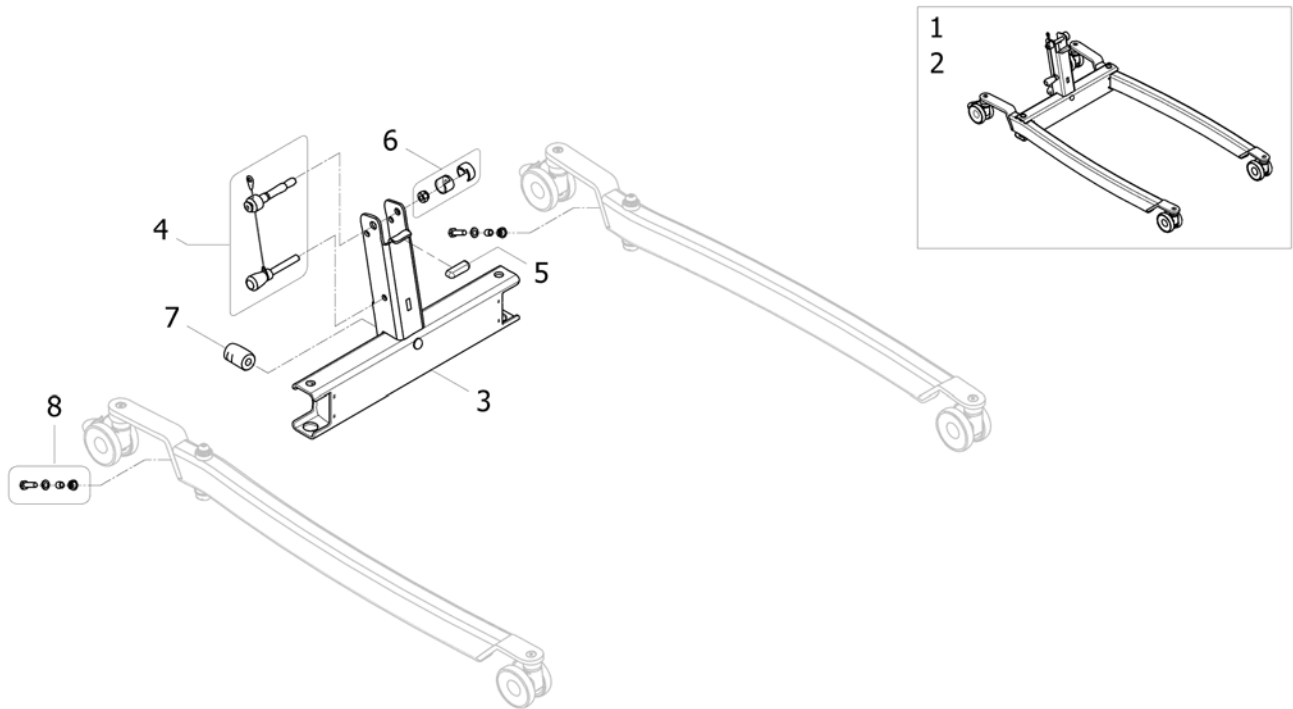Base for Foldable Mast

ISPS1650623 ~ 1646693


Pos.	Part number	Description	Valid from → until	Qty
1	SP1646693	Base and legs fully assembled with 75mm castors		1
2	SP1645811	Base and legs fully assembled with 100mm castors		1
3	SP1645796	Base for foldable mast for Birdie EVO COMPACT		1
4	SP1646694	Handle kit for foldable mast		1
5	SP1545308-7012	Padded insert plastic		1
6	SP1650834	Plastic cover kit for foldable mast		1
7	SP1510217-7012	Pedal axle for Birdie lifters		1
8	SP1650835	Fixation kit for connection between leg and rod		1

Base for Fixed Mast

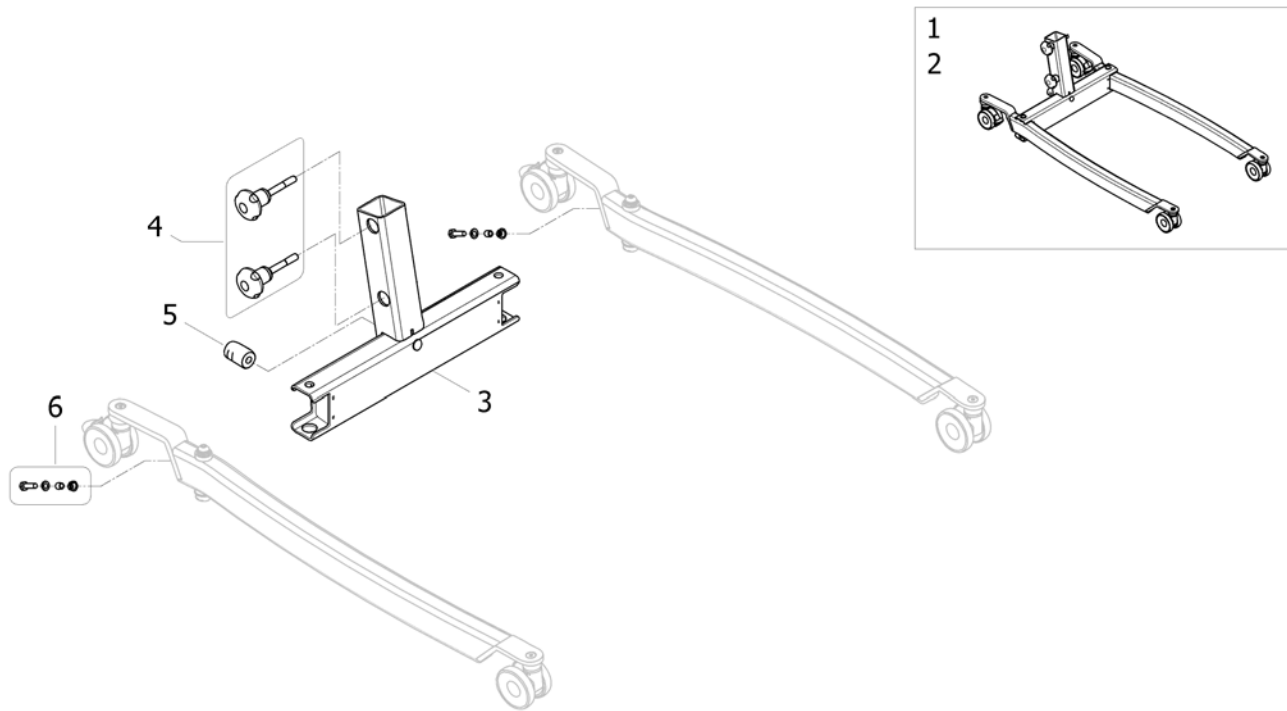

ISPS1650727 ~ 1646695



Pos.	Part number	Description	Valid from → until	Qty
1	SP1646695	Base and legs fully assembled with 75mm castors		1
2	SP1645815	Base and legs fully assembled 100mm castors		1
3	SP1645809	Base for fixed mast for Birdie EVO COMPACT		1
4	SP1650836	Knob fixation kit		1
5	SP1510217-7012	Pedal axle for Birdie lifters		1
6	SP1650835	Fixation kit for connection between leg and rod		1

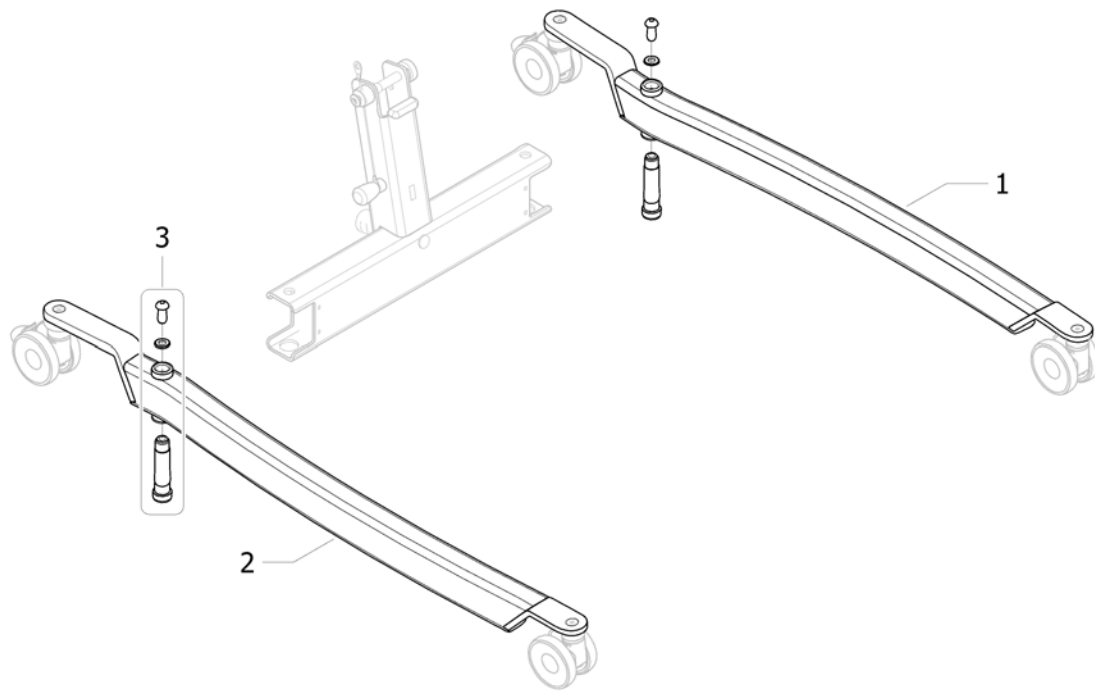Leg assembly

ISPS1650728 ~ 1646693


Pos.	Part number	Description	Valid from → until	Qty
1	SP1650837	Left leg for Birdie EVO COMPACT		1
2	SP1650838	Right leg for Birdie EVO COMPACT		1
3	SP1650839	Leg fixation kit, bushing, screw and washer		1

Pedal mechanism assembly

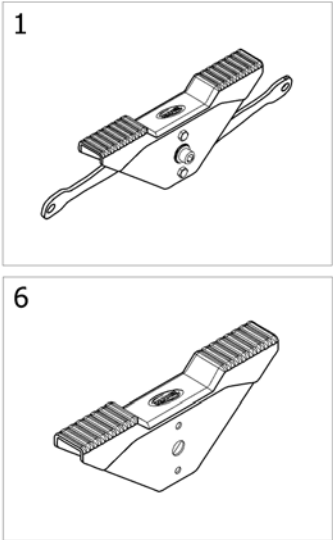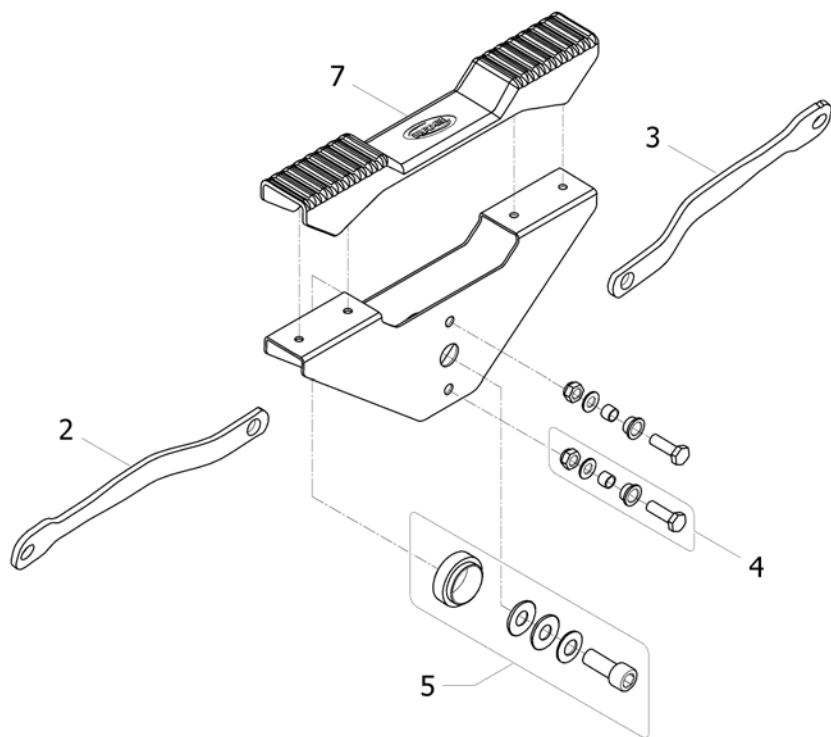

ISPS1650729 ~ 1646185


Pos.	Part number	Description	Valid from → until	Qty
1	SP1646185	Pedal mechanism fully assembled		1
2	SP1645791	Left connecting rod		1
3	SP1645790	Right connecting rod (with dent)		1
4	SP1650840	Fixation kit for connection between pedal and rod		1
5	SP1650841	Fixation kit for connection for pedal and base		1
6	SP1650842	Pedal kit (plate and cover)		1
7	SP1645588	Pedal cover		1

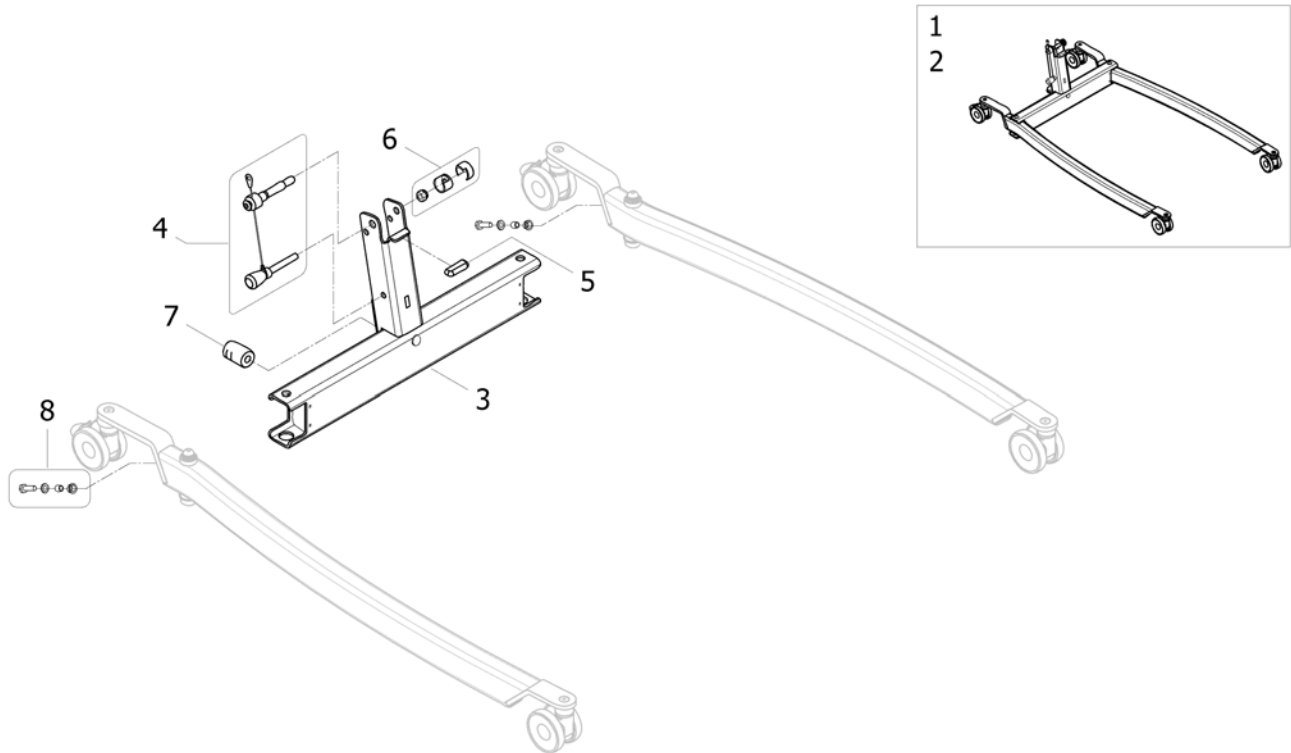Base for Foldable Mast

ISP1650731 ~ 1646819

Pos.	Part number	Description	Valid from → until	Qty
1	SP1646819	Base and legs fully assembled with 75mm castors		1
2	SP1645714	Base and legs fully assembled with 100mm castors		1
3	SP1645501	Base for foldable mast for Birdie EVO		1
4	SP1646694	Handle kit for foldable mast		1
5	SP1545308-7012	Padded insert plastic		1
6	SP1650834	Plastic cover kit for foldable mast		1
7	SP1510217-7012	Pedal axle for Birdie lifters		1
8	SP1650835	Fixation kit for connection between leg and rod		1


Base for Fixed Mast

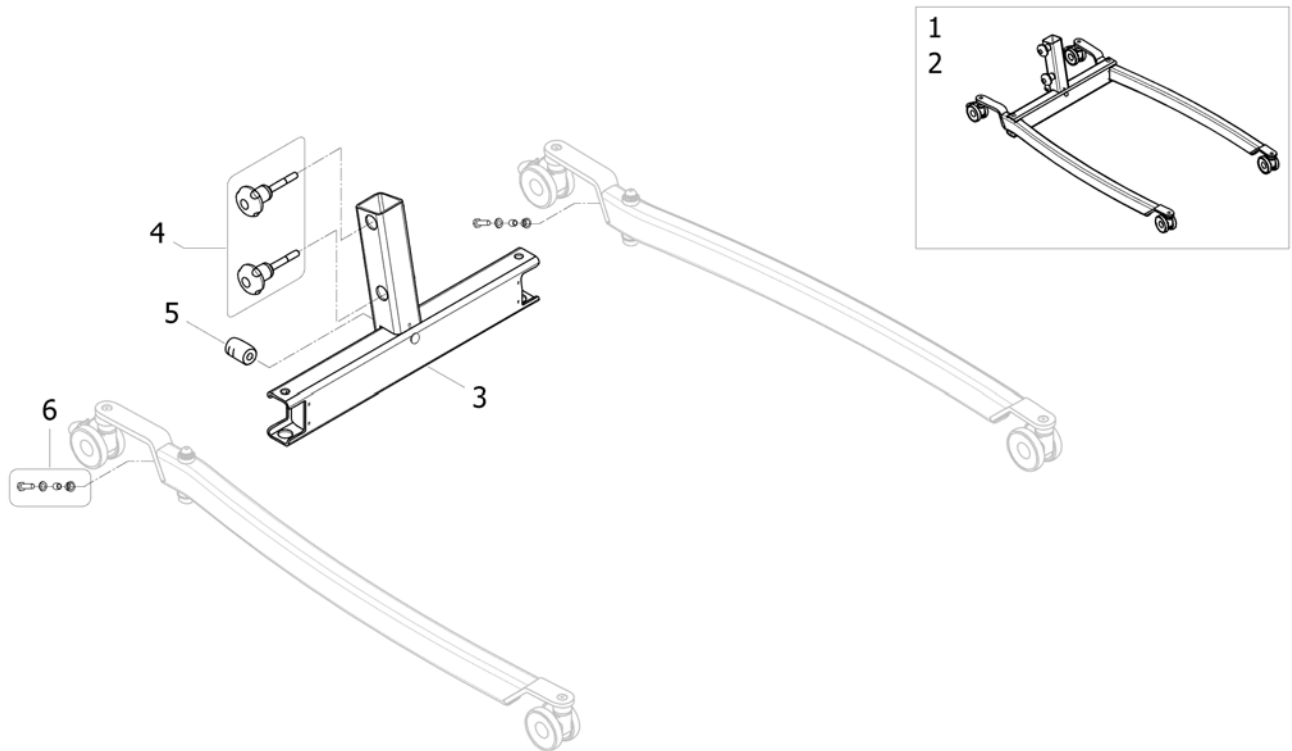

ISP1650732 ~ 1646696

Pos.	Part number	Description	Valid from → until	Qty
1	SP1646696	Base and legs fully assembled with 75mm castors		1
2	SP1645721	Base and legs fully assembled with 100mm castors		1
3	SP1645722	Base for fixed mast for Birdie EVO		1
4	SP1650836	Knob fixation kit		1
5	SP1510217-7012	Pedal axle for Birdie lifters		1
6	SP1650835	Fixation kit for connection between leg and rod		1



Leg assembly

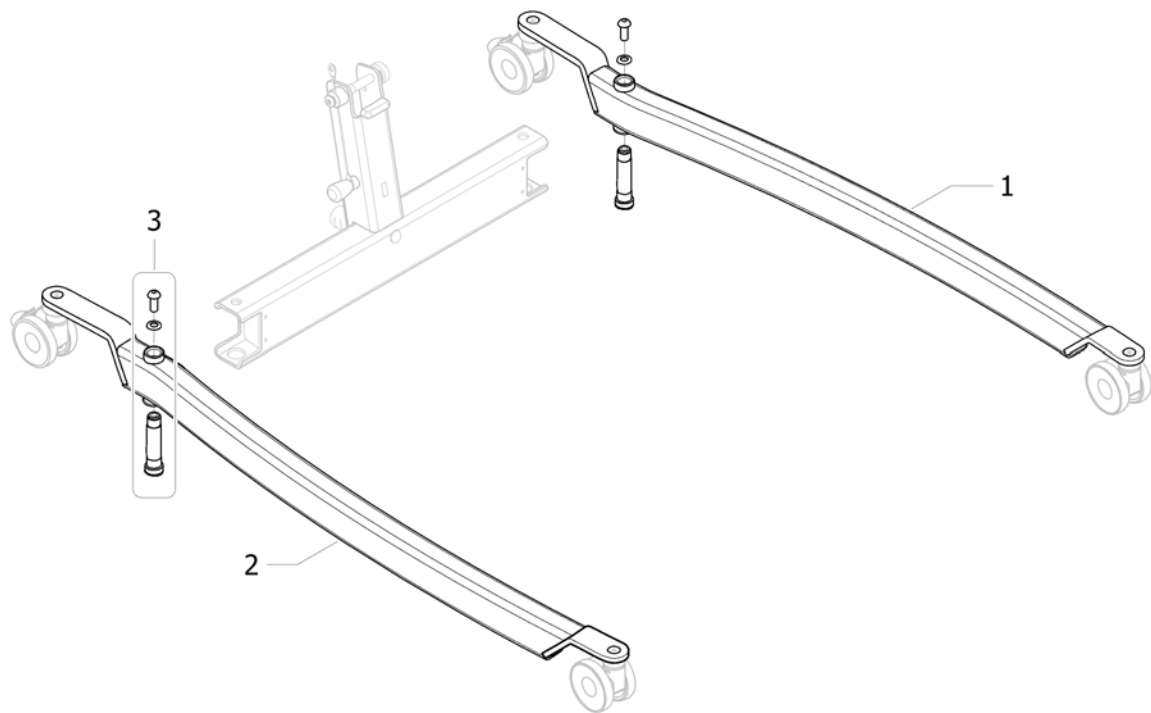

ISPS1650733 ~ 1646819

Pos.	Part number	Description	Valid from → until	Qty
1	SP1650844	Left leg for Birdie EVO		1
2	SP1650845	Right leg for Birdie EVO		1
3	SP1650839	Leg fixation kit, bushing, screw and washer		1


Pedal mechanism assembly

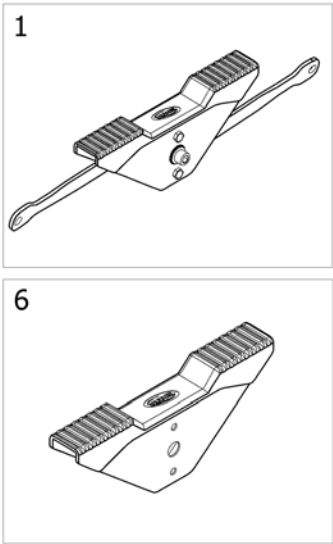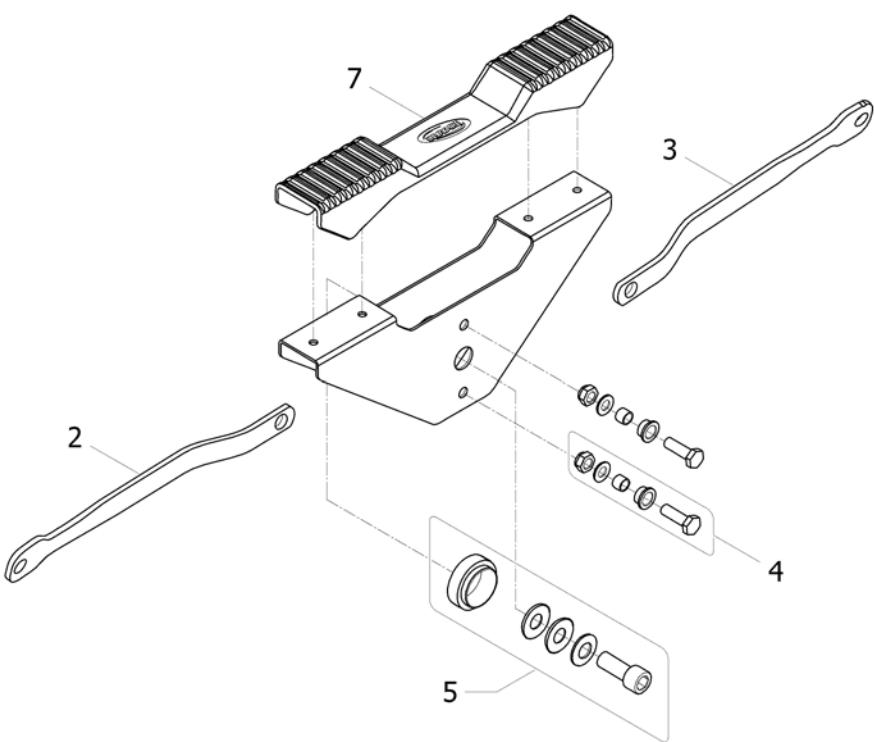

ISPS1650734 ~ 1645715

Pos.	Part number	Description	Valid from → until	Qty
1	SP1645715	Pedal mechanism fully assembled		1
2	SP1645512	Left connecting rod		1
3	SP1645507	Right connecting rod (with dent)		1
4	SP1650840	Fixation kit for connection between pedal and rod		1
5	SP1650841	Fixation kit for connection for pedal and base		1
6	SP1650842	Pedal kit (plate and cover)		1
7	SP1645588	Pedal cover		1


Base for Foldable Mast

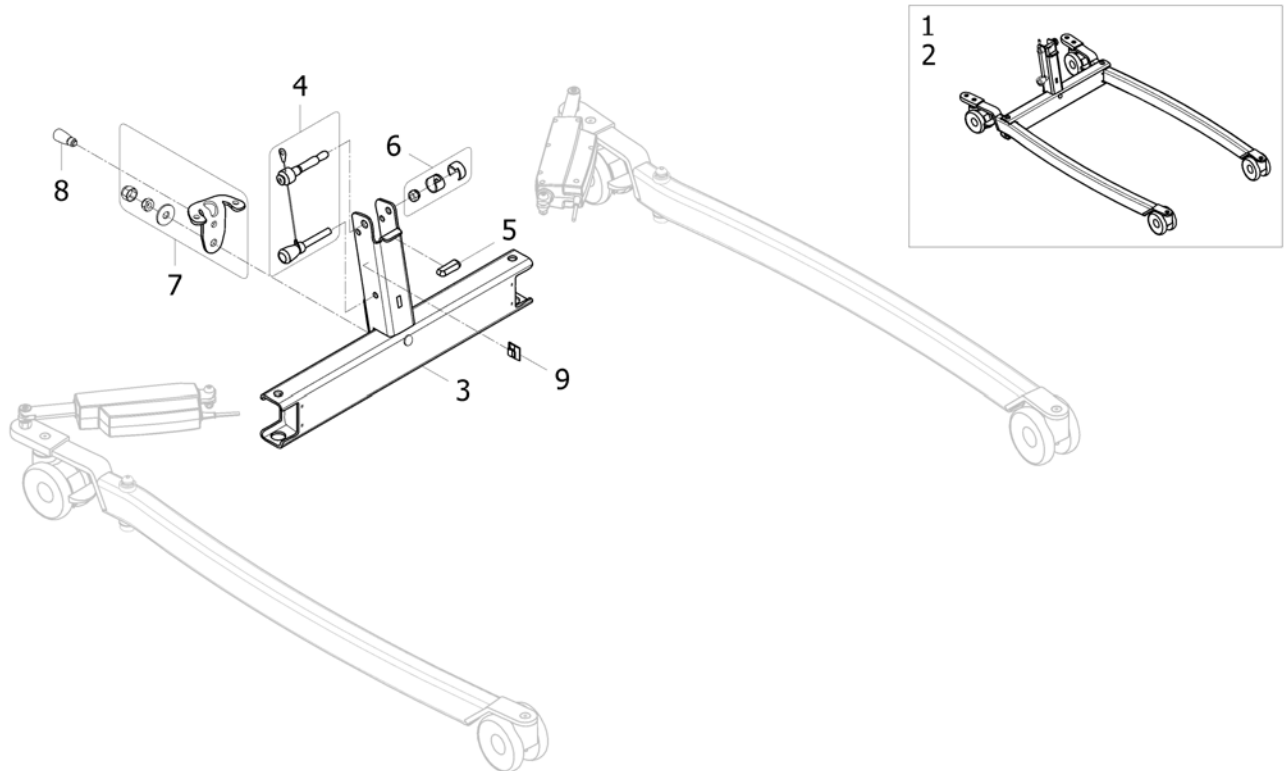

ISP1650736 ~ 1646820


Pos.	Part number	Description	Valid from → until	Qty
1	SP1646697	Base and legs fully assembled for Birdie EVO elec. with 75mm castors		1
2	SP1645820	Base and legs fully assembled for Birdie EVO elec. with 100mm castors		1
3	SP1645501	Base for foldable mast for Birdie EVO		1
4	SP1646694	Handle kit for foldable mast		1
5	SP1545308-7012	Padded insert plastic		1
6	SP1650834	Plastic cover kit for foldable mast		1
7	SP1651009	Base plate fixation kit		1
8	SP1651184	Add-on for upright positioning		1
9	SP1426680	Cable clips 4,6x4,1 Qty.2		1

Leg assembly

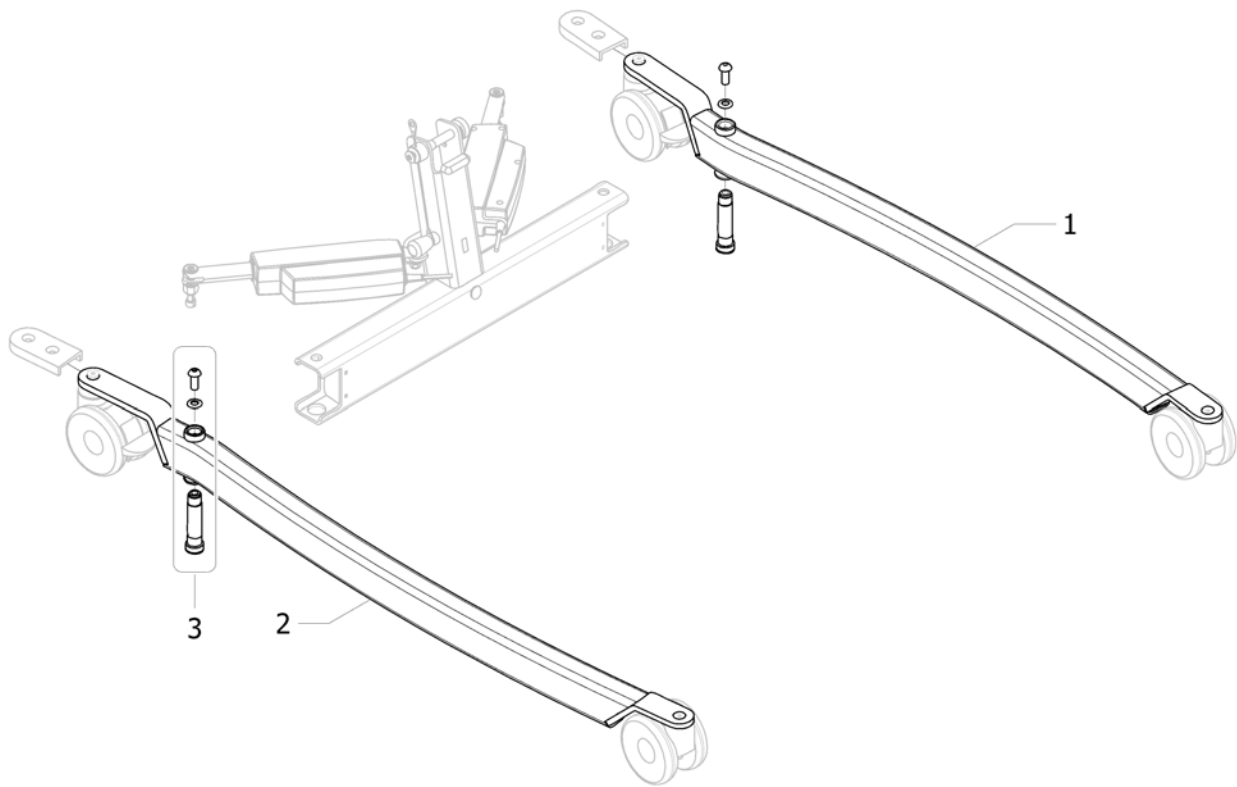

ISPS1650819 ~ 1645820


Pos.	Part number	Description	Valid from → until	Qty
1	SP1650844	Left leg for Birdie EVO		1
2	SP1650845	Right leg for Birdie EVO		1
3	SP1650839	Leg fixation kit, bushing, screw and washer		1

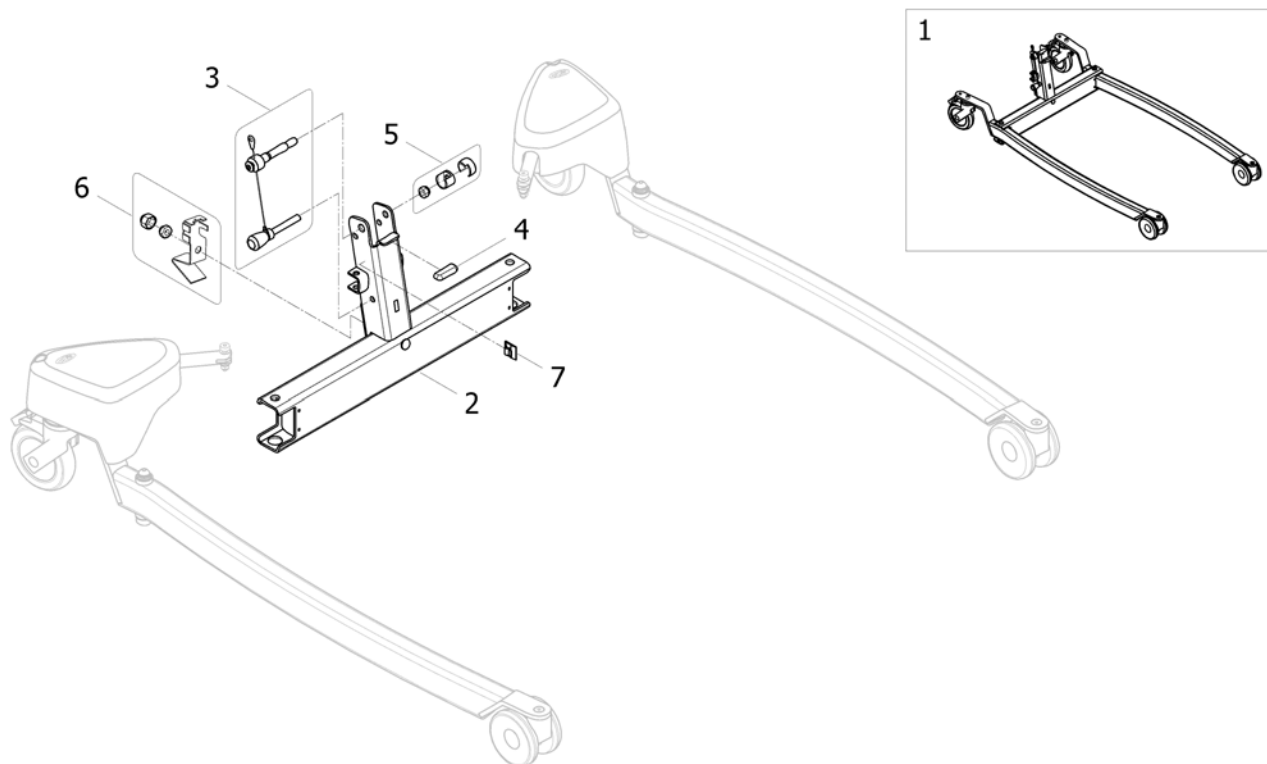Base for Foldable Mast

ISP1650821 ~ 1645846

Pos.	Part number	Description	Valid from → until	Qty
1	SP1645846	Base and legs fully assembled, Birdie EVO PLUS		1
2	SP1646788	Base for foldable mast for Birdie EVO PLUS		1
3	SP1646694	Handle kit for foldable mast		1
4	SP1545308-7012	Padded insert plastic		1
5	SP1650834	Plastic cover kit for foldable mast		1
6	SP1651198	Base fixation kit		1
7	SP1426680	Cable clips 4,6x4,1 Qty.2		1


Base for Fixed Mast

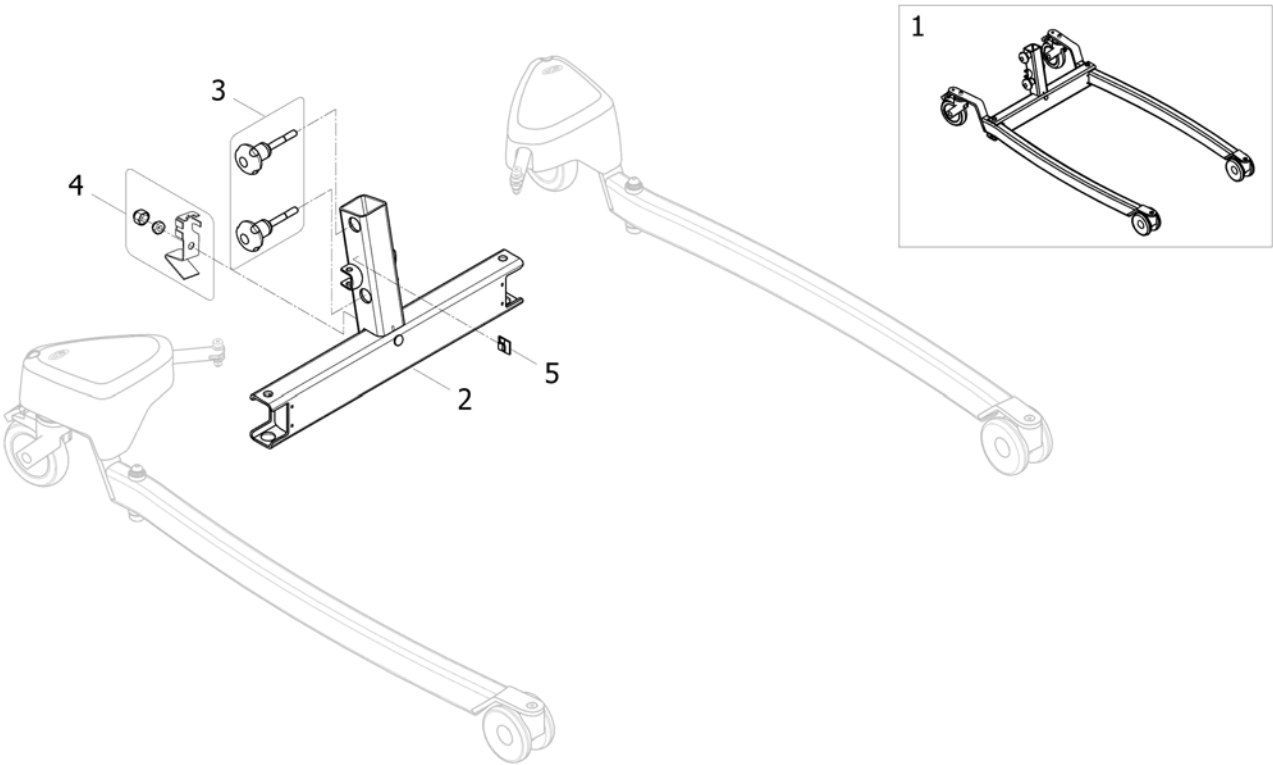

ISP1650822 ~ 1646828

Pos.	Part number	Description	Valid from → until	Qty
1	SP1646828	Base and legs fully assembled, Birdie EVO PLUS		1
2	SP1646829	Base for fixed mast for Birdie EVO PLUS		1
3	SP1650836	Knob fixation kit		1
4	SP1651198	Base fixation kit		1
5	SP1426680	Cable clips 4,6x4,1 Qty.2		1


Leg assembly

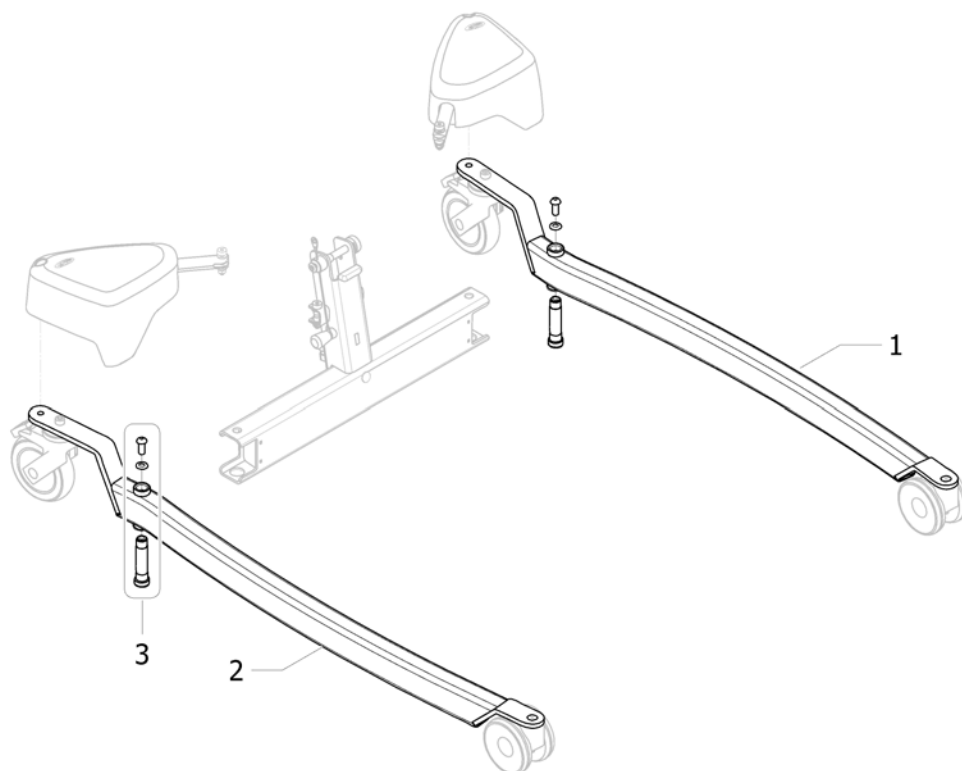

ISPS1650823 ~ 1645846


Pos.	Part number	Description	Valid from → until	Qty
1	SP1651199	Left leg for Birdie EVO PLUS		1
2	SP1651200	Right leg for Birdie EVO PLUS		1
3	SP1650839	Leg fixation kit, bushing, screw and washer		1

Birdie EVO COMPACT and Birdie EVO castors

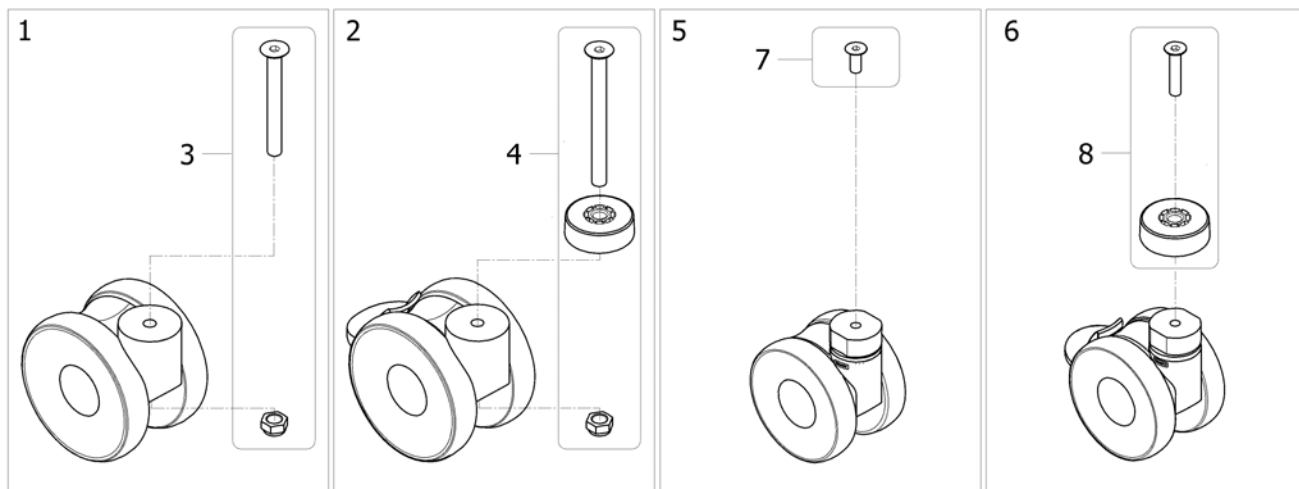

ISPS1650824


Pos.	Part number	Description	Valid from → until	Qty
1	SP1650846	Front castor 100mm without brake, fixation included		1
2	SP1650847	Rear castor 100mm with brake, fixation included		1
3	SP1650848	Fixation kit for 100mm front castor		1
4	SP1650849	Fixation kit for 100mm rear castor		1
5	SP1650850	Front castor 75mm without brake, fixation included		1
6	SP1650851	Rear castor 75mm with brake, fixation included		1
7	SP1426737	Screw M8x20 Qty.2		1
8	SP1650852	Fixation kit for 75mm rear castor		1

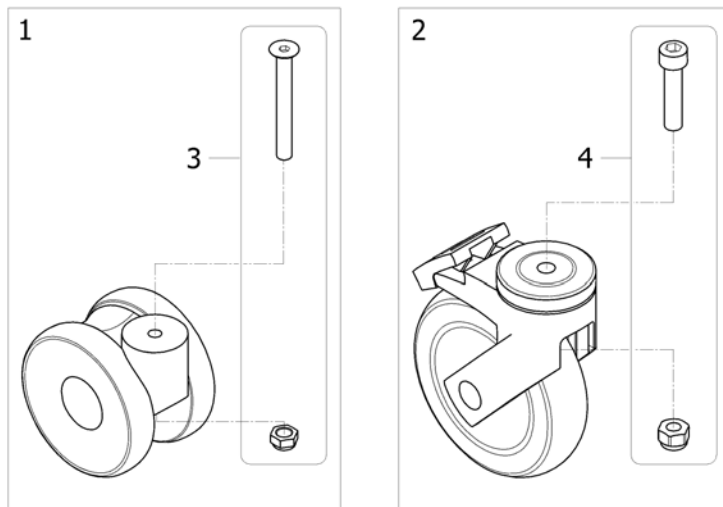

ISPS1650864

Pos.	Part number	Description	Valid from → until	Qty
1	SP1650846	Front castor 100mm without brake, fixation included		1
2	SP1650853	Rear castor 125mm with brake, fixation included		1
3	SP1650848	Fixation kit for 100mm front castor		1
4	SP1650854	Fixation kit for 125mm rear castor		1


Actuator for Boom

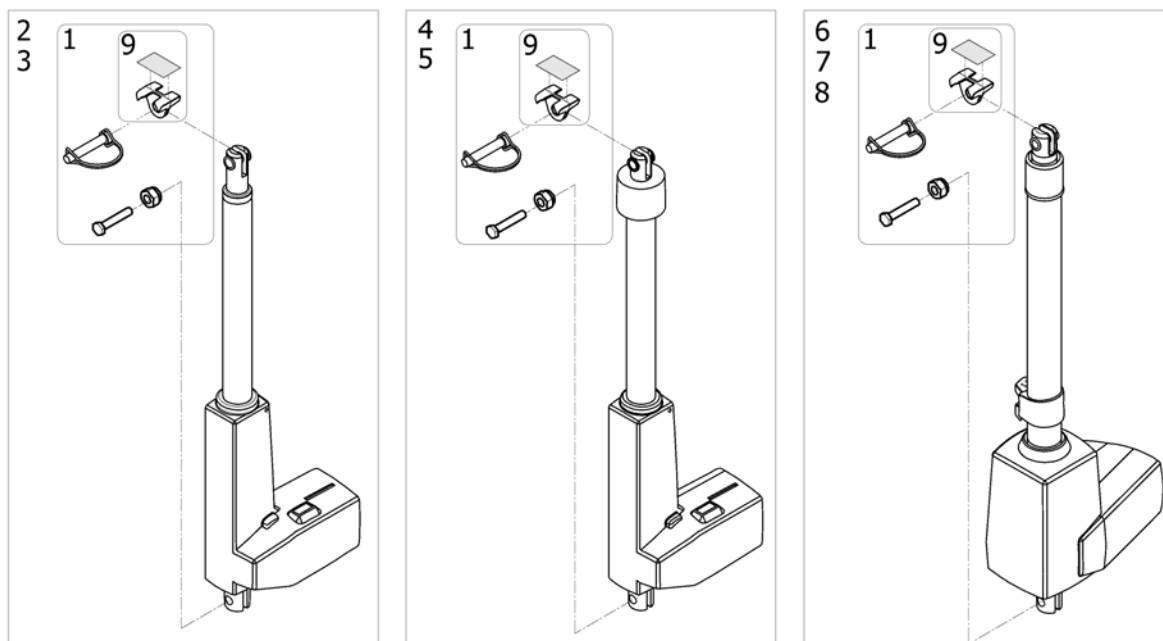

ISPS1650827

Pos.	Part number	Description	Valid from → until	Qty
1	SP1650855	Fixation kit for boom actuator		1
9	SP1645767	Birdie EVO anti-pinching plastic		1
2	SP1650856	Actuator for Birdie EVO COMPACT LA31 HiLo Jack		1
3	SP1650999	Actuator for Birdie EVO COMPACT LA31 HiLo MiniFit		1
4	SP1651001	Actuator for Birdie EVO COMPACT with emergency lowering LA31 HiLo Jack		1
5	SP1651002	Actuator for Birdie EVO COMPACT with emergency lowering LA31 HiLo MiniFit		1
6	SP1651003	Actuator for Birdie EVO LA34 HiLo Jack Detachable battery/control unit		1
7	SP1651539	Actuator for Birdie EVO LA34 HiLo MiniFit Fixed battery/control unit		1
8	SP1651005	Actuator for Birdie EVO PLUS LA34 HiLo Jack		1


Actuator for elec. leg spread, Birdie EVO

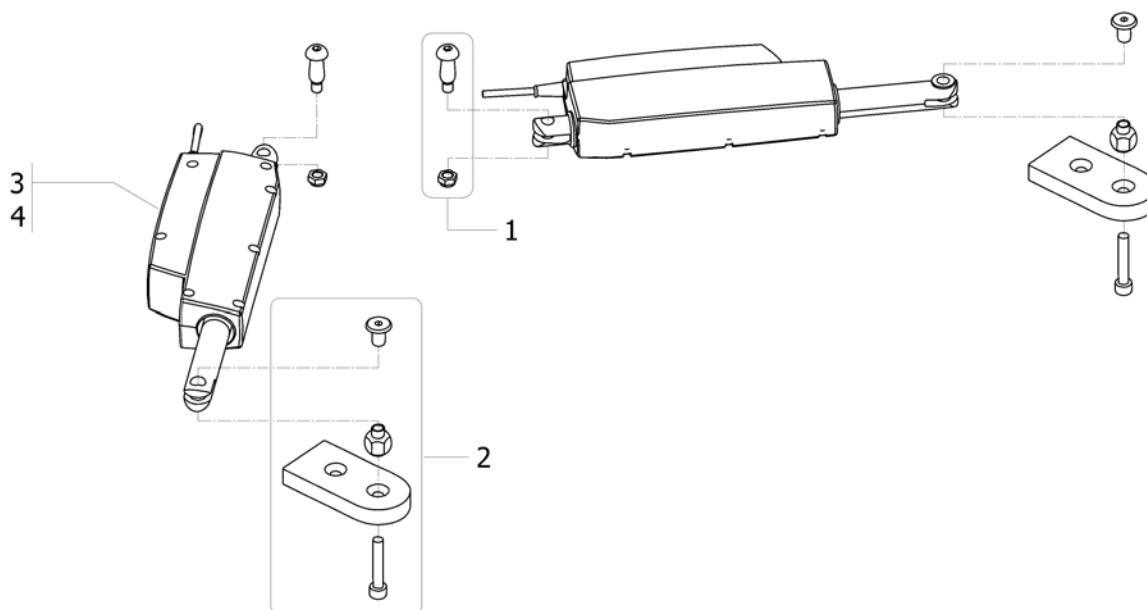

ISPS1650828 ~ 1645820




Pos.	Part number	Description	Valid from → until	Qty
1	SP1651201	Fixation kit for connection between Actuator and base		1
2	SP1651202	Fixation kit for connection between Actuator and leg		1
3	SP1646691-9005	Actuator LA12 x2Pcs with Cable Birdie Minifit		1
4	SP1648547-9005	Actuator LA12 x2Pcs with Cable Birdie Jack		1

Actuator for elec. leg spread, Birdie EVO PLUS

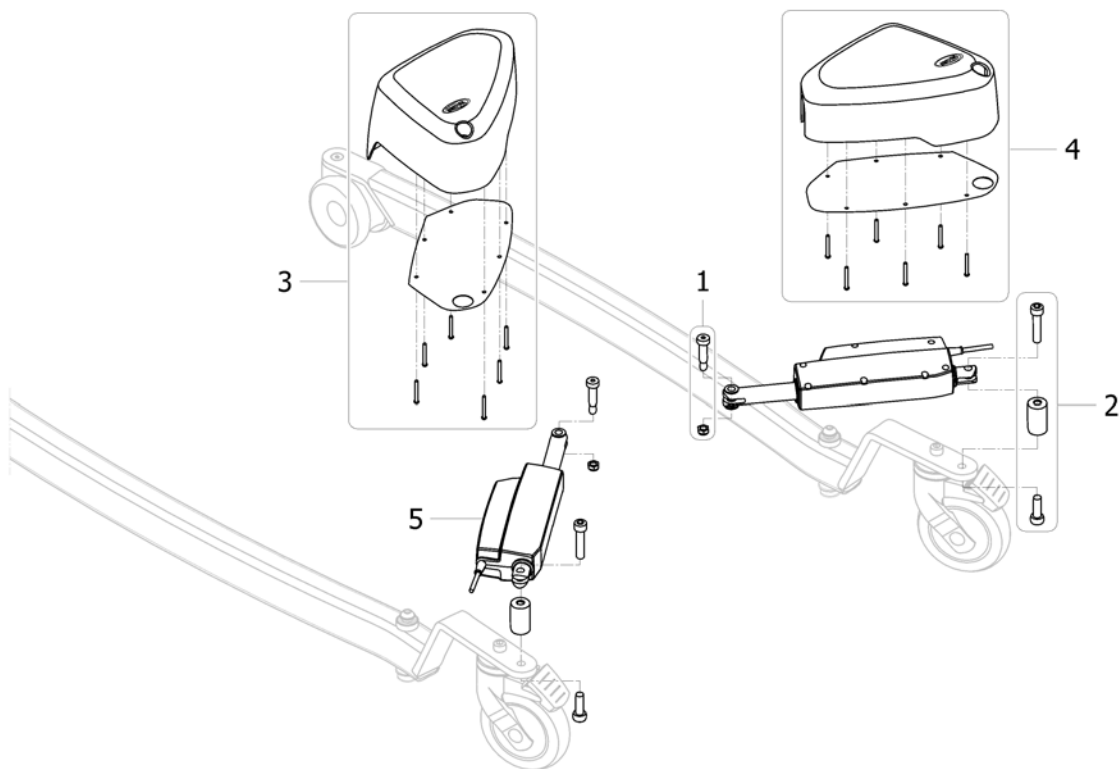

ISPS1650829 ~ 1645846

Pos.	Part number	Description	Valid from → until	Qty
1	SP1651231	Fixation kit for connection between Actuator and base		1
2	SP1651232	Fixation kit for connection between Actuator and leg		1
3	SP1651233	Left Cover Kit for leg spread Actuator		1
4	SP1651234	Right Cover Kit for leg spread Actuator		1
5	SP1648539-9005	Actuator LA12 x2Pcs with Cable Birdie Jack		1

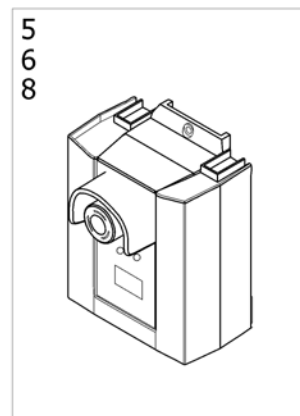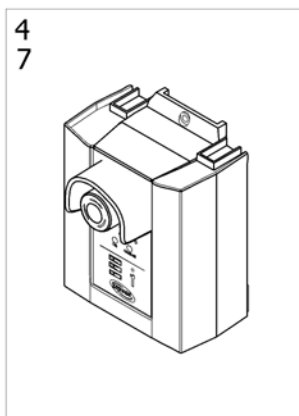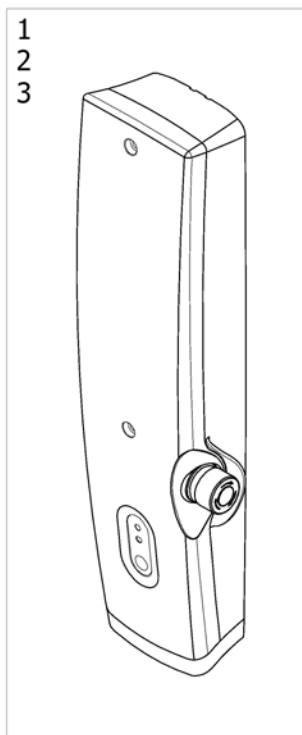
Control units

Control unit	Part number	Birdie EVO version	Nr of channels	Hand control part number
CBJ-Home	1 SP1575046-7035	Birdie EVO	1	SP1509617
	3 SP1575044-7035	Birdie EVO COMPACT		
	2 SP1575048-7035	Birdie EVO	2	SP1509618
CBJ2	5 SP1648546-7035	Birdie EVO	1	SP1574990
	8 SP1651006-7035	Birdie EVO COMPACT		
CBJ1	6 SP1651004-7035	Birdie EVO / Birdie EVO PLUS	2	SP1522195
CBJ-Care	4 SP1574997-7035	Birdie EVO	1	SP1574990
	7 SP1574998-7035	Birdie EVO	2	SP1155872
		Birdie EVO PLUS		SP1177596

ISPS1650830

Pos.	Part number	Description	Valid from → until	Qty
1	SP1575046-7035	Control unit 1 Channel CBJ-Home manual leg 180Kg		1
2	SP1575048-7035	Control unit 2 Channel CBJ-Home electrical leg 180Kg		1
3	SP1575044-7035	Control unit 1 Channel CBJ-Home manual leg 150Kg		1
4	SP1574997-7035	Control unit 1 Channel CBJ-Care manual leg 180Kg		1
5	SP1648546-7035	Control unit CBJ2 manual leg spread 180Kg		1
6	SP1651004-7035	Control unit CBJ1 electrical leg spread 180Kg		1
7	SP1574998-7035	Control unit 2 Channels CBJ-Care electrical leg 180Kg		1
8	SP1651006-7035	Control unit CBJ2 manual leg spread 150Kg		1


Hand controls

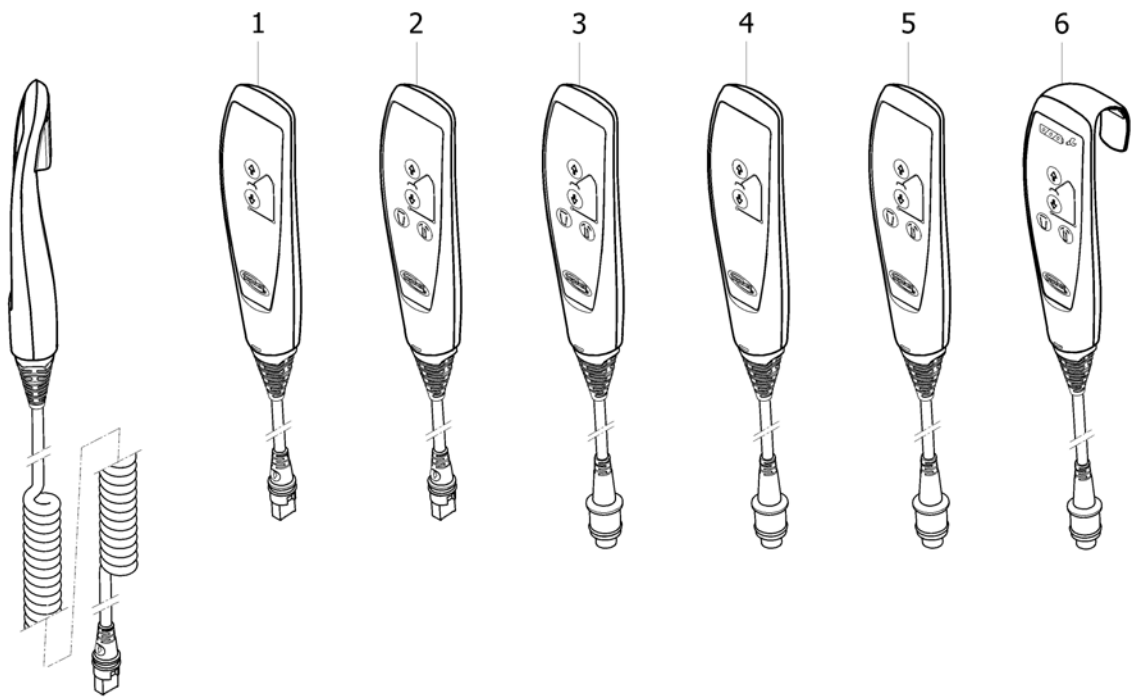

ISPS1650831

Pos.	Part number	Description	Valid from → until	Qty
1	SP1509617	Handcontrol with magnet, 1 Channel Manual for Birdie EVO or Birdie EVO COMPACT		1
2	SP1509618	Handcontrol with magnet, 2 Channels for Birdie EVO		1
3	SP1522195	Handcontrol with magnet, 2 Channels for Birdie EVO or Birdie EVO PLUS		1
4	SP1574990	Handcontrol with magnet, 1 Channel for Birdie EVO or Birdie EVO COMPACT		1
5	SP1155872	Handcontrol with magnet, 2 Channels Birdie EVO		1
6	SP1177596	Handcontrol with hook, 2 Channels for Birdie EVO		1


Electrical Accessories & Hardware

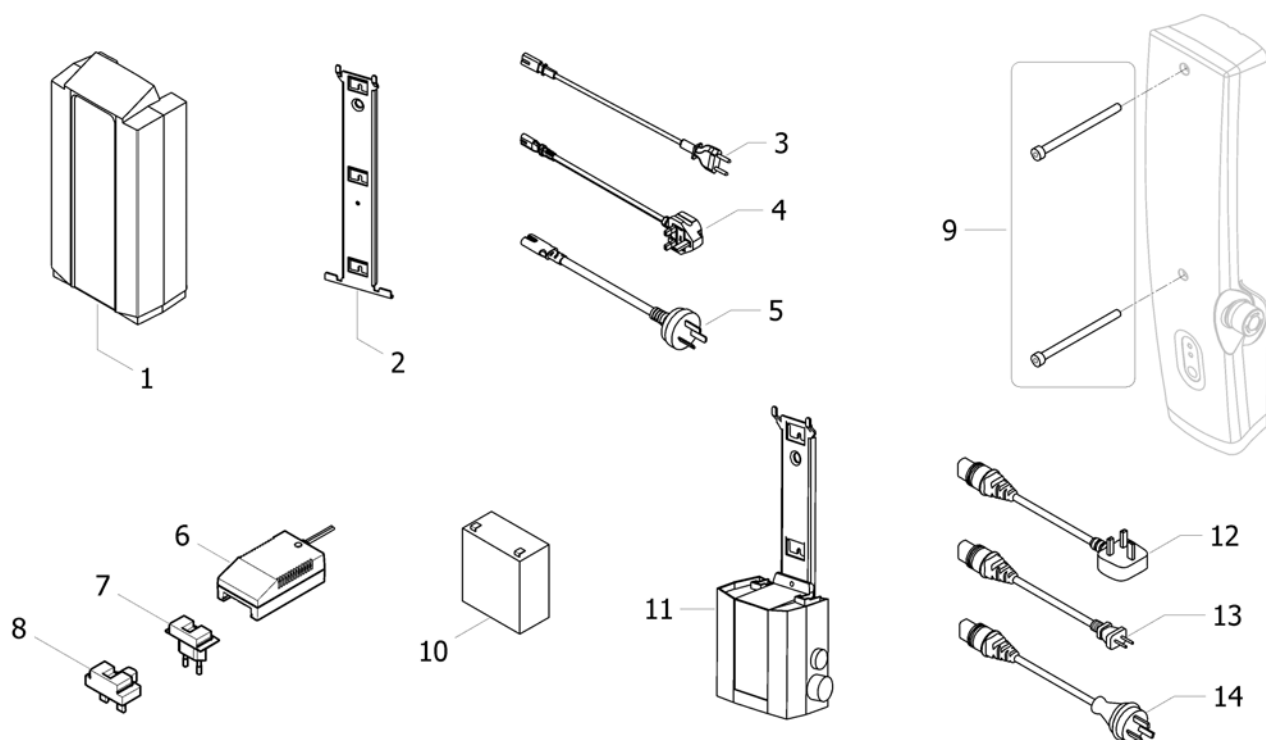

ISP51650032

Pos.	Part number	Description	Valid from → until	Qty
1	SP1514573-7035	Battery c/w housing CBJ control units RAL 7035 (Linak)		1
2	SP1514574-0002	Bracket for battery housing and CBJ control units (Linak)		1
3	SP1511583-7035	Mains cable, EU plug		1
4	SP1511686-7035	Mains Cable, UK plug		1
5	SP1530000-7035	Mains cable, ANZ plug		1
6	SP1514576-9005	Charger CH01, Birdie EVO / Birdie EVO COMPACT (1 pc per hoist) please, order the adaptor		1
7	SP1514613-9005	EU plug adaptor for charger		1
8	SP1533127-9005	UK plug adaptor for charger		1
9	SP1509740	Screws for CBJ-Home		1
10	SP1420622	Battery 2,9 Ah (1 pcs)		1
11	SP1651491	Wall charger Kit for detachable battery		1
12	SP1079134	Wall Charger Main Cable, UK		1
13	SP1079135	Wall Charger Main Cable, EU		1
14	SP1417620	Wall Charger Main Cable, ANZ		1


Birdie EVO COMPACT, Mast and Boom, Foldable or Fixed

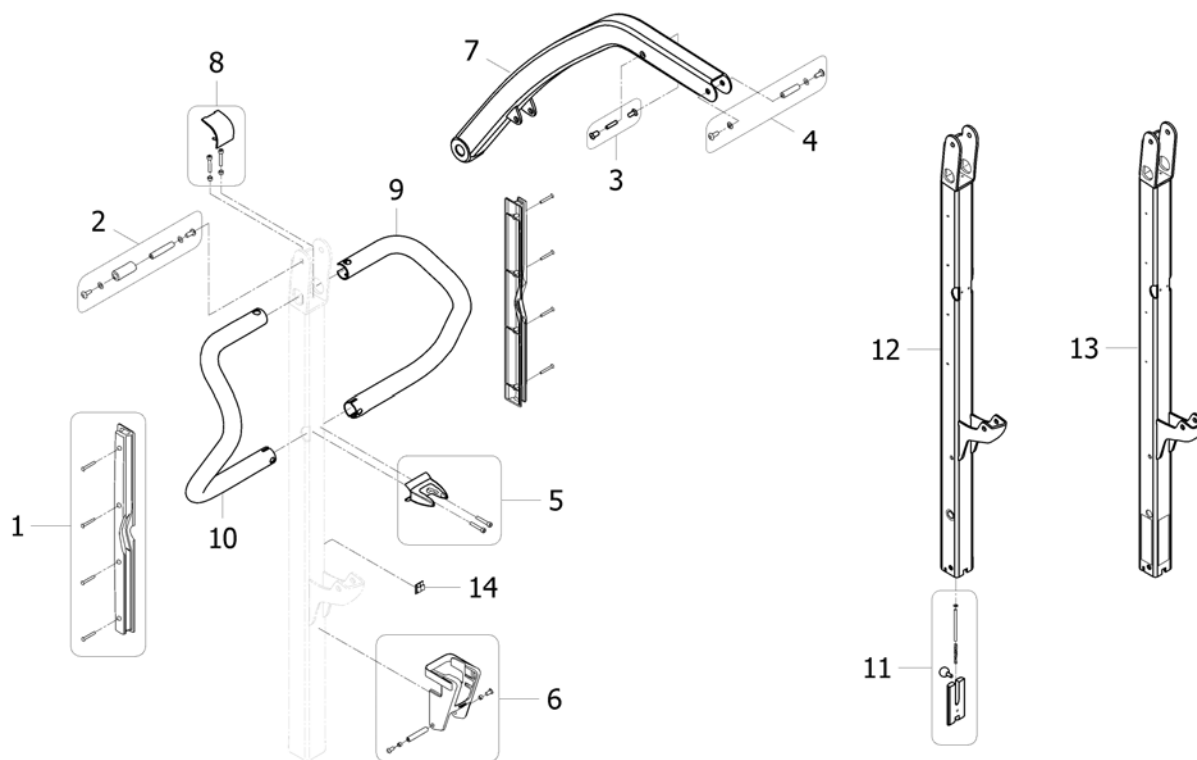

ISPS1651242 ~ 1655812

Pos.	Part number	Description	Valid from → until	Qty
1	SP1651253	Cable Cover kit		1
2	SP1651254	Mast/Boom Articulation Kit		1
3	SP1651255	Damper fixation kit		1
4	SP1651256	Spreaderbar Articulation kit		1
5	SP1651257	Actuator Holder kit		1
6	SP1651258	Spreaderbar Holder kit		1
7	SP1651328	Boom for Birdie EVO COMPACT		1
8	SP1651270	Push bar fixation kit		1
9	SP1651329	Left Push bar, Birdie EVO COMPACT		1
10	SP1651330	Right Push bar, Birdie EVO COMPACT		1
11	SP1651273	Locking kit for foldable Mast		1
12	SP1651331	Foldable Mast for Birdie EVO COMPACT		1
13	SP1651332	Fixed Mast for Birdie EVO COMPACT		1
14	SP1426680	Cable clips 4,6x4,1 Qty.2		1


Birdie EVO and EVO PLUS, Mast and Boom, Foldable or Fixed

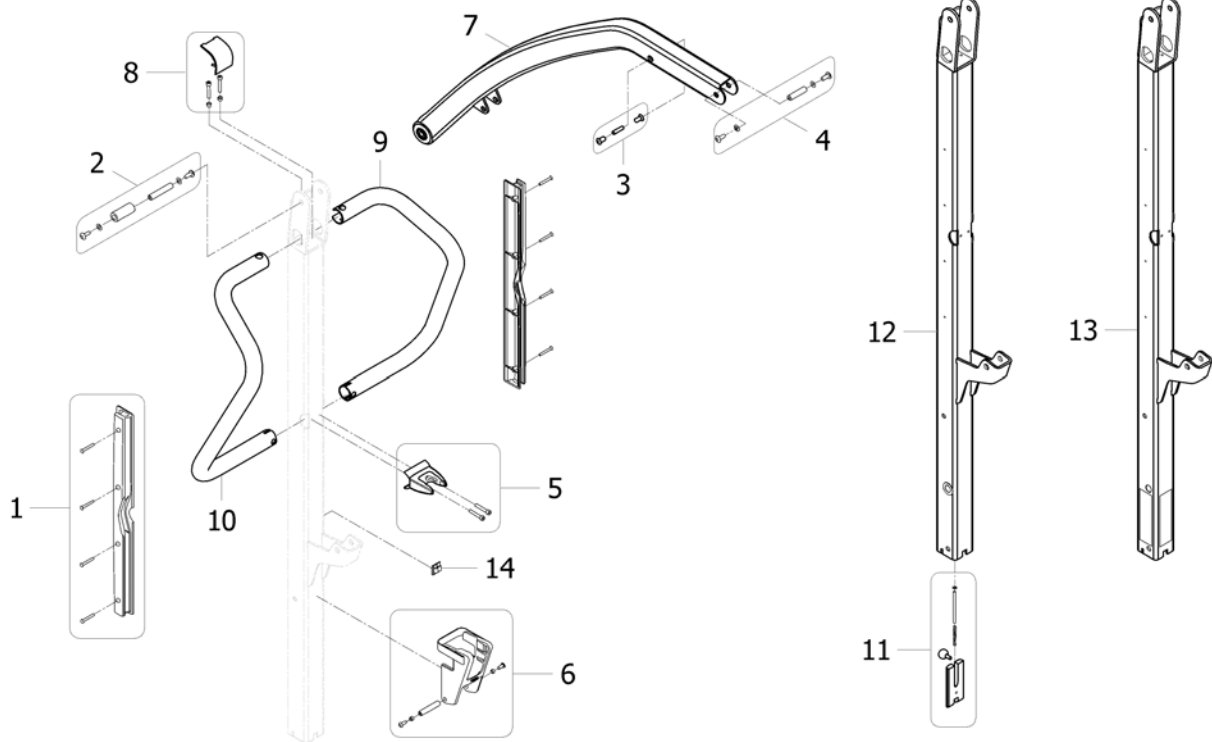

SPS1651243 ~ 1645716

Pos.	Part number	Description	Valid from → until	Qty
1	SP1651253	Cable Cover kit		1
2	SP1651254	Mast/Boom Articulation Kit		1
3	SP1651255	Damper fixation kit		1
4	SP1651256	Spreaderbar Articulation kit		1
5	SP1651257	Actuator Holder kit		1
6	SP1651258	Spreaderbar Holder kit		1
7	SP1651259	Birdie EVO Boom		1
8	SP1651270	Push bar fixation kit		1
9	SP1651271	Left Push bar		1
10	SP1651272	Right Push bar		1
11	SP1651273	Locking kit for foldable Mast		1
12	SP1651312	Foldable Mast for Birdie EVO		1
13	SP1651313	Fixed Mast for Birdie EVO		1
14	SP1426680	Cable clips 4,6x4,1 Qty.2		1


Labelling, Birdie EVO

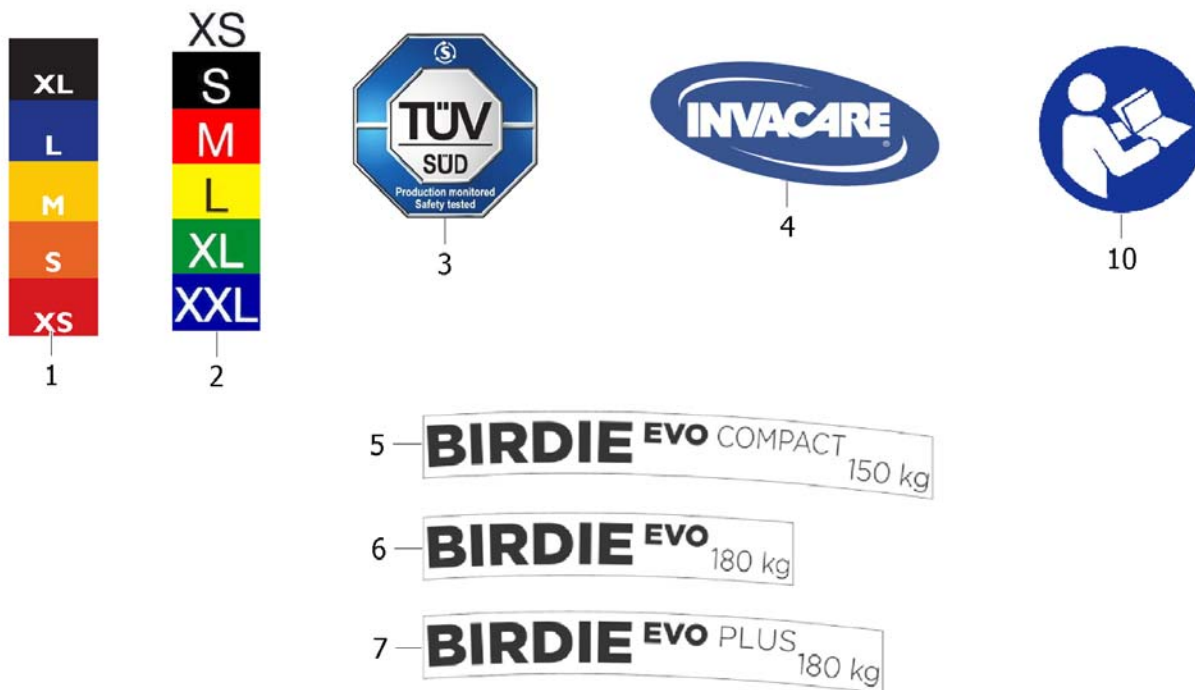

ISPS1651252

Pos.	Part number	Description	Valid from → until	Qty
1	SP1511359	Label with sling sizes		1
2	SP1530013	Label with sling sizes for Australia/New Zealand		1
3	SP1514228	Label TUV Certification		1
4	SP417547	Label Invacare Logo		1
5	SP1646411	Label Birdie EVO COMPACT 150kg Qty.2		1
6	SP1646410	Label Birdie EVO 180kg Qty.2		1
7	SP1646774	Label Birdie EVO PLUS 180kg Qty.2		1
10	SP1582519	Label Read User Manual Qty.2		1


Spreader Bar Damper Slow'R

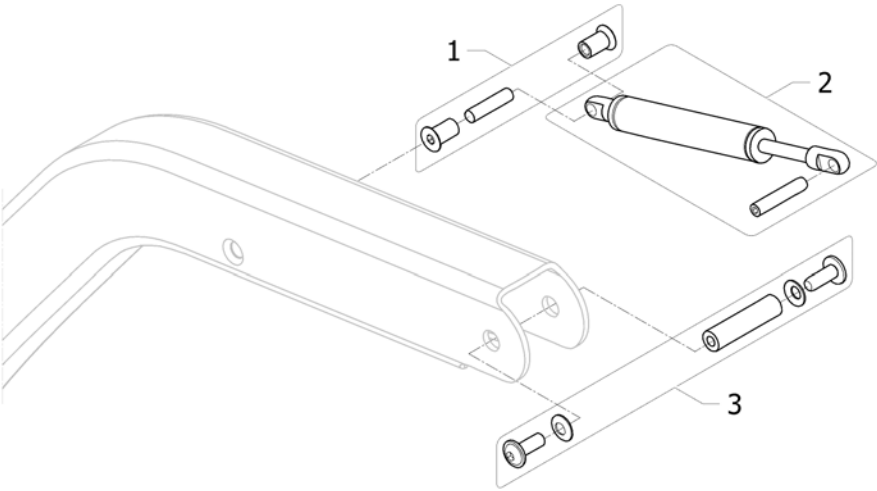

ISP51651338


Pos.	Part number	Description	Valid from → until	Qty
1	SP1651255	Damper fixation kit		1
2	AP1646419	Damper Slow'R, Kit		1
3	SP1651256	Spreaderbar Articulation kit		1

Spreader bar Hardware

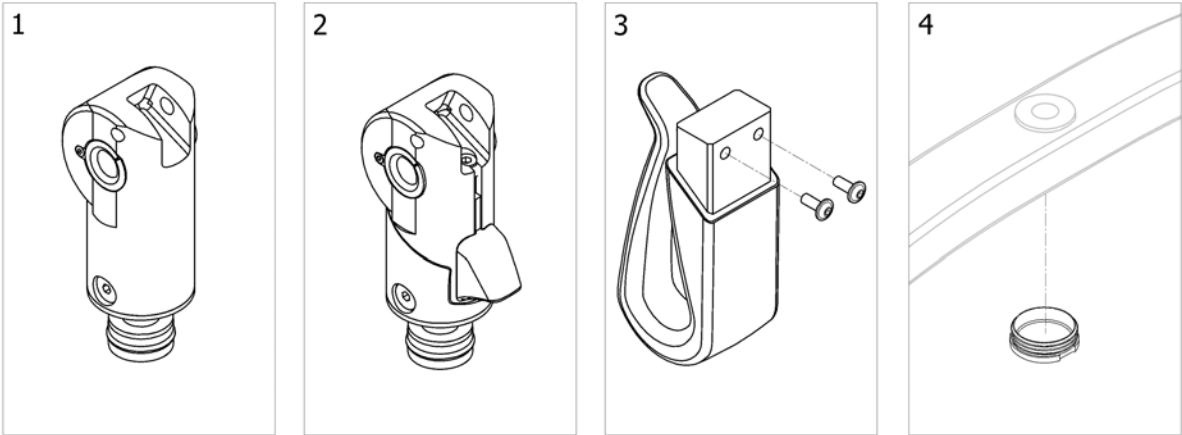

ISPS1651335


Pos.	Part number	Description	Valid from → until	Qty
1	SP1645769	Connection kit for fixed spreader bar		1
2	AP1645728	Connection kit for detachable spreader bar (SMARTLOCK)		1
3	SP1645828	Spreader bar hook kit		1
4	SP1645734	Spreader bar plastic cap		1

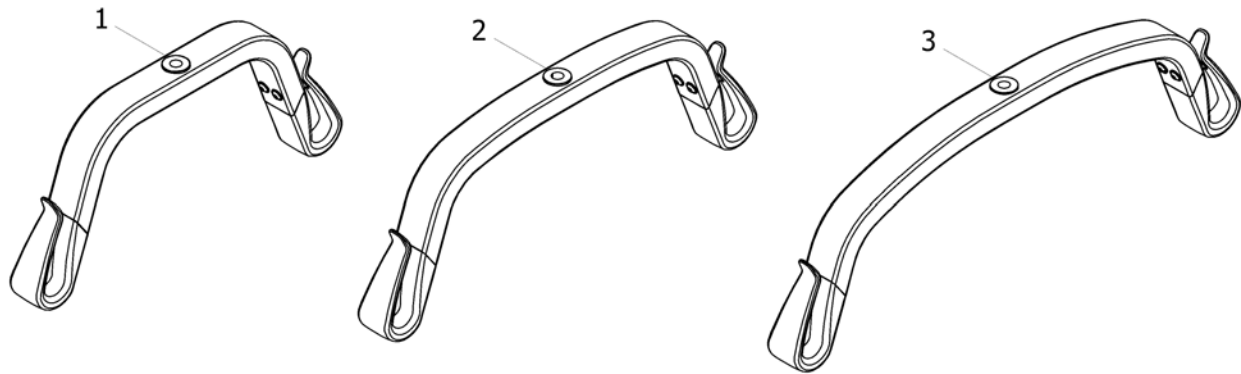Spreader bar 2 points (350/450/550 mm)

ISPS1651336

Pos.	Part number	Description	Valid from → until	Qty
1	SP1645731	Spreader bar 2pts 350 mm without fixation		1
2	SP1645742	Spreader bar 2pts 450 mm without fixation		1
3	SP1645744	Spreader bar 2pts 550 mm without fixation		1


Spreader bar 4 points (450/550 mm)

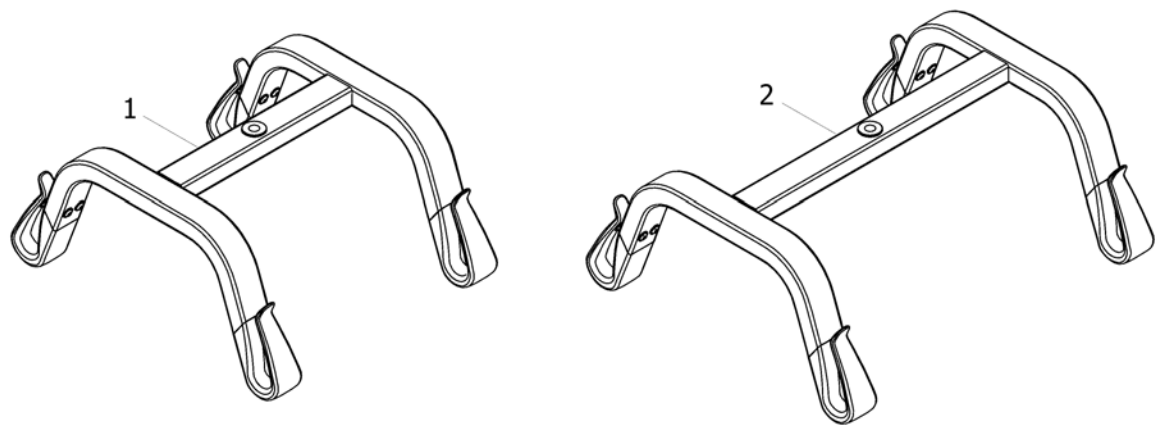

ISPS1651337

Pos.	Part number	Description	Valid from → until	Qty
1	SP1645747	Spreader bar 4pts 450 mm without fixation		1
1	SP1645753	Spreader bar 4pts 550 mm without fixation		1



Scale and Attachment

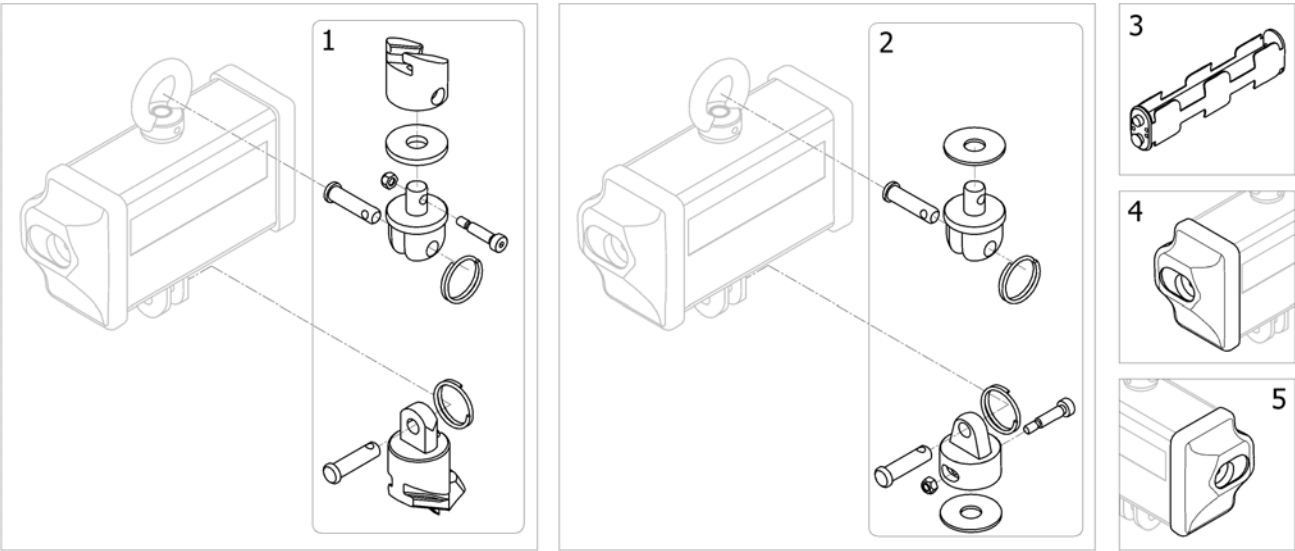

ISPS1651369

Pos.	Part number	Description	Valid from → until	Qty
1	SP1651537	Scale attachment kit for SMARTLOCK mechanism		1
2	SP1651538	Scale attachment kit for Fixed mechanism		1
3	SP1641530	Scale Battery holder		1
4	SP1557840	Scale End Cap Left		1
5	SP1557839	Scale End Cap Right		1



Lever

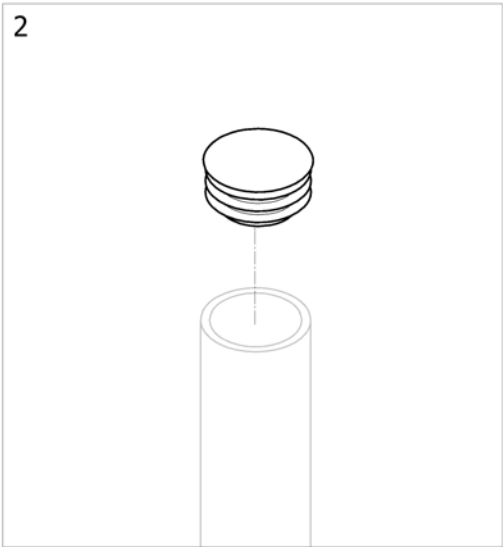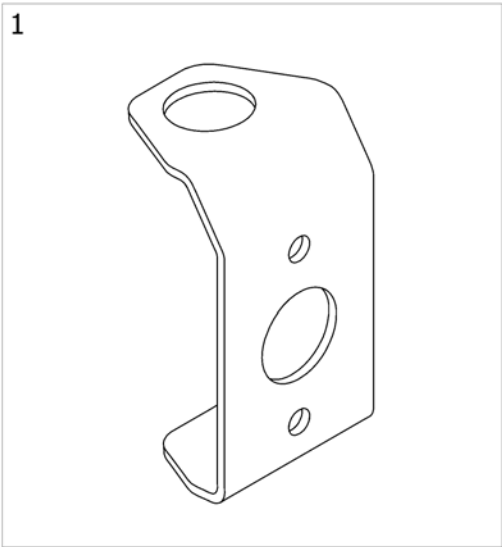

ISPS1651339

Pos.	Part number	Description	Valid from → until	Qty
1	SP1645498	Fixation plate for leg spread lever		1
2	SP1449580	Plastic plug for leg spread lever		1

Leg Protection

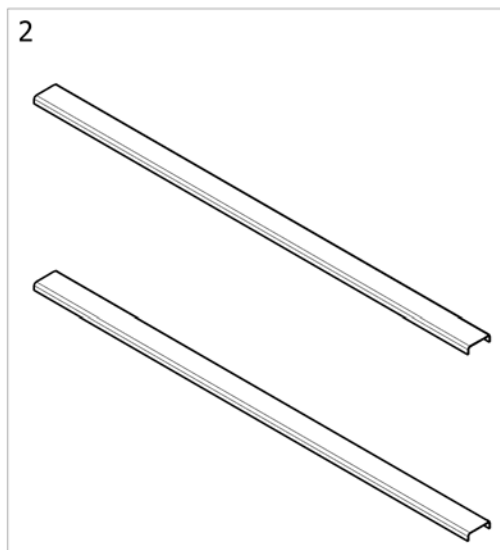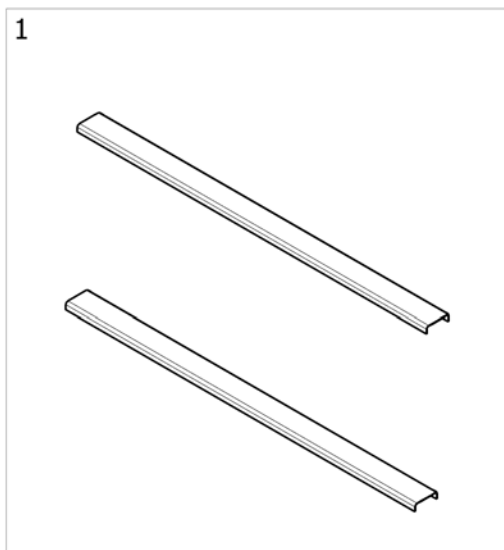

ISPS1651356

Pos.	Part number	Description	Valid from → until	Qty
1	SP1648554	Leg Protection for Birdie EVO COMPACT Qty.2		1
2	SP1648551	Leg Protection for Birdie EVO, EVO PLUS Qty.2		1



Yes, you can.®



Belgium & Luxemburg:

Invacare nv, Autobaan 22, B-8210 Loppem
Tel: (32) (0)50 83 10 10, Fax: (32) (0)50 83 10 11
belgium@invacare.com / www.invacare.be

Denmark:

Invacare A/S, Sdr. Ringvej 37, DK-2605 Brøndby
Tel: (45) (0)36 90 00 00, Fax: (45) (0)36 90 00 01
denmark@invacare.com / www.invacare.dk

Deutschland:

Invacare Aquatec GmbH, Alemannenstraße 10, D-88316 Isny
Tel: (49) (0)75 62 7 00 0, Fax: (49) (0)75 62 7 00 66
info@invacare-aquatec.com / www.invacare-aquatec.de

European Distributor Organisation:

Invacare, Kleiststraße 49, D-32457 Porta Westfalica
Tel: (49) (0)57 31 754 540, Fax: (49) (0)57 31 754 541
edo@invacare.com / www.invacare.eu.com

España:

Invacare SA, c/Areny s/n, Polígono Industrial de Celrà, E-17460 Celrà (Girona)
Tel: (34) (0)972 49 32 00, Fax: (34) (0)972 49 32 20
contactsp@invacare.com / www.invacare.es

France:

Invacare Poirier SAS, Route de St Roch, F-37230 Fondettes
Tel: (33) (0)2 47 62 64 66, Fax: (33) (0)2 47 42 12 24
contactfr@invacare.com / www.invacare.fr

Ireland:

Invacare Ireland Ltd, Unit 5 Seatown Business Campus, Seatown Road, Swords, County Dublin - Ireland
Tel: (353) 1 810 7084, Fax: (353) 1 810 7085
ireland@invacare.com / www.invacare.ie

Italia:

Invacare Mecc San s.r.l., Via dei Pini 62, I-36016 Thiene (VI)
Tel: (39) 0445 38 00 59, Fax: (39) 0445 38 00 34
italia@invacare.com / www.invacare.it

Nederland:

Invacare BV, Celsiusstraat 46, NL-6716 BZ Ede
Tel: (31) (0)318 695 757, Fax: (31) (0)318 695 758
nederland@invacare.com, csede@invacare.com / www.invacare.nl

Norge:

Invacare AS, Grensesvingen 9, Postboks 6230, Etterstad, N-0603 Oslo
Tel: (47) (0)22 57 95 00, Fax: (47) (0)22 57 95 01
norway@invacare.com, island@invacare.com / www.invacare.no

Österreich:

Invacare Austria GmbH, Herzog Odilostrasse 101, A-5310 Mondsee
Tel: (43) 6232 5535 0, Fax: (43) 6232 5535 4
info-austria@invacare.com / www.invacare.at

Portugal:

Invacare Lda, Rua Estrada Velha, 949, P-4465-784 Leça do Balio
Tel: (351) (0)225 1059 46/47, Fax: (351) (0)225 1057 39
portugal@invacare.com / www.invacare.pt

Sverige & Suomi:

Invacare AB, Fagerstagatan 9, S-163 91 Spånga
Tel: (46) (0)8 761 70 90, Fax: (46) (0)8 761 81 08
sweden@invacare.com, finland@invacare.com / www.invacare.se

Switzerland:

Invacare AG, Benkenstrasse 260, CH-4108 Witterswil
Tel: (41) (0)61 487 70 80, Fax: (41) (0)61 487 70 81
switzerland@invacare.com / www.invacare.ch

United Kingdom:

Invacare Limited, Pencoed Technology Park, Pencoed, Bridgend CF35 5AQ
Switchboard Tel: (44) (0)1656 776200, Fax: (44) (0)1656 776201
Customer services Tel: (44) (0)1656 776222, Fax: (44) (0)1656 776220
uk@invacare.com / www.invacare.co.uk