



ALLMOBILITY TRADING S.r.l.

Operative headquarters: Via G. Balla, 4 42124 Reggio Emilia (RE) - ITALY

Registered head office: Via Mentana, 150 43100 Parma (PR) - ITALY

Tel. 0522-941078

E-mail: info@allmobility.it

Website: www.allmobility.it

"BUFFALO 150" SIIRTYMÄLAUTA



Product complying with the requirements of Directive 2007/47/EC.

KÄYTTÖOHJEET
Version 001-16

"BUFFALO 150" BOARD

DIMENSIONS	WEIGHT (Kg)	BEARING CAPACITY (Kg)
<p>  77 cm  32,5 cm  0,4 cm LENGTH = 77cm WIDTH = 32.5cm THICKNESS = 0.4cm </p>	<p>1.23</p>	<p>150</p> <p><i>distributed over a surface of 40 cm, with resting weight of 25 kg on the sides</i></p>

STORAGE TEMPERATURE

ADMISSIBLE TEMPERATURE RANGE:

MINIMUM = -5 °C

MAXIMUM = +40 °C

3. GENERAL INFORMATION

The board has been designed and made to be safe in the use for which it is intended, as long as it is used and stored as indicated in these instructions.

Given below are the main instructions and correct storage by the user to as to reduce situations of risk.



= PAY ATTENTION!



= FORBIDDEN ACTION



The board is a medical aid for facilitating the transfer of a disabled user from a wheelchair to another seat.

DO NOT USE THE BOARD FOR ANY PURPOSE OTHER THAN THE ONE DESCRIBED IN CHAPTER 2.



DO NOT USE THE BOARD IF IT SHOWS ANY SIGN OF DAMAGE, DEFORMATION, CRACKS, THINNING, BREAKAGES.



DO NOT USE THE BOARD TO TRANSFER WEIGHTS/PERSONS HEAVIER THAN THE MAXIMUM ALLOWED BEARING CAPACITY (150 Kg).



**DO NOT USE CORROSIVE PRODUCTS TO CLEAN THE BOARD.
ALL IT NEEDS IS A DAMP CLOTH, SURGICAL SPIRIT OR ORDINARY SOAP.**



FOLLOW THE INSTRUCTIONS FOR USE PROVIDED ON THE BOARD AND IN CHAPTER 4.



MAKE SURE THAT THE DISTANCE TO THE POINT OF ARRIVAL IS IDEAL FOR A CORRECT AND STEADY POSITIONING OF THE BOARD.



WHEN POSSIBLE, AS WELL AS ONTO THE ARM OF THE WHEELCHAIR, HOOK THE BOARD ALSO ONTO A SUPPORT ON THE POINT OF ARRIVAL.






IF THE INSTRUCTIONS ON THE LABEL APPLIED ON THE BOARD ARE SO WORN THAT THEY ARE NO LONGER LEGIBLE, ASK THE MANUFACTURER FOR A NEW COPY.




4. INSTRUCTIONS FOR USE

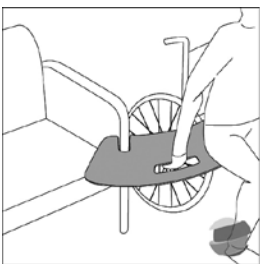
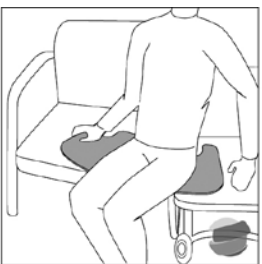
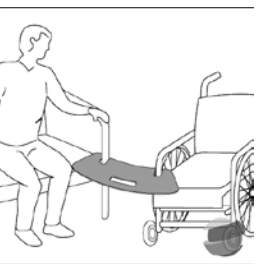
The *BUFFALO 150* board is a support for transferring the user from the wheelchair to another position, for example to the bed, couch, car, etc.


It does not have any packaging, so it is bought, used, kept and carried just as it is.

Shown below are some of the most frequent uses and how to perform them, as shown also on the label applied on the accessory.

			<p><u>TRANSFER TO THE CAR</u></p> <p>Get as close as possible to the seat, secure the board to the arm of the wheelchair and ensure that it is resting firmly on the seat.</p>
---	---	---	---

			<p><u>TRANSFER TO THE BED</u></p> <p>Get as close as possible to the bed, secure the board to the arm of the wheelchair and ensure that it is resting firmly on the mattress.</p>
--	--	--	--

			<p><u>TRANSFER TO A CHAIR/COUCH</u></p> <p>Get as close as possible to the chair/couch, secure the board to the arm of the wheelchair and to the armrest of the chair/couch (if possible) and ensure that it is resting firmly on the seat.</p>
---	---	---	--

			<p><u>TRANSFER TO THE TOILET</u></p> <p>Get as close as possible to the toilet, secure the board to the arm of the wheelchair and ensure that it is resting firmly on the seat.</p>
---	---	---	--

ALWAYS FOLLOW THE INSTRUCTIONS IN CHAPTER 3.

5. DISPOSAL

When the board is no longer to be used, dispose of it correctly without dumping it in the environment.
Follow the rules for disposal in force in the place of use of the accessory (ask the local authorised disposal companies).

6. WARRANTY - ASSISTANCE

Listed below are the general terms of the warranty applied by the Manufacturer **ALLMOBILITY TRADING S.r.l.** .

The warranty applies to all manufacturing and material defects, duly confirmed at the Company's discretion.

The Warranty does not cover damage caused by:

- *Normal wear and tear*
- *Deterioration of parts with purely aesthetic functions*
- *Repairs or modifications carried out by unauthorised persons or centres*
- *Use of the product not complying with the warnings given in this manual*
- *Damage caused by unsuitability of the environment in which the product is used and by phenomena not depending on its normal operation.*

In case of sale the warranty has a duration of 1 (one) year.

For any problem or to request information, contact the Manufacturer at the addresses on the first page.

