


GTM-MUSTANG

PARTS LIST

Element	Page
Standard aluminium / titanium footrest	01
Individual aluminium / titanium footrest	02
Standard plastic footplatform	03
Individual plastic or carbon fiber footplatform	04
Castors housing	05
Plastic castor 4" x 26 mm for two-arm fork	06
Plastic castor 4" x 32 mm for two-arm fork	07
Aluminum caster 4" x 35 mm for two-arm fork	08
Aluminum castor 4" x 35 mm with sealed bearings for two-arm forks	09
Plastic castor 4" x 26 mm for monoarm fork	10
Plastic castor 4" x 32 mm for monoarm fork	11
Aluminum castor 4" x 35 mm for monoarm fork	12
Aluminum castor 4" x 35 mm with sealed bearings for monoarm forks	13
Plastic castor 5" x 29 mm for two-arm fork	14
Plastic castor 5" x 27 mm for two-arm fork	15
Aluminum castor 5" x 27 mm for two-arm fork	16
Aluminum caster 5" x 35 mm for two-arm fork	17
Aluminum castor 5" x 35 mm with sealed bearings for two-arm forks	18
Plastic castor 5" x 29 mm for monoarm fork	19
Aluminum castor 5" x 35 mm for monoarm fork	20
Aluminum castor 5" x 35 mm with sealed bearings for monoarm forks	21
Removable sideguard for 44 cm rear seat height	22
Removable sideguard for 42 cm rear seat height	23
Removable sideguard for 40 cm rear seat height	24
Sideguard folding with the backrest	25
Removable armrest	26
Scissor brakes with clamps	27
Side brakes with clamps	28
Front monounting point for removable sideguard	29
Left mounting point of folding backrest mechanism	30
Right mounting point of folding backrest mechanism	31
Right side backrest mounting point of folding backrest mechanism for removable sideguards	32
Left side backrest mounting point of folding backrest mechanism for removable sideguards	33
Backrest handle mounting system	34
Aluminum pushhandle	35
Backrest without pushhandles	36
Backrest height adjusting system	37
Backrest with plastic foldable pushhandles	38
Backrest with screw-in pushhandles	39
Backrest with integral pushhandles	40
Backrest with pushhandles in backrest tubes	41
Camberbar clamps without antitipp receiver	42

[illegible]

Standard aluminum / titanium footrest

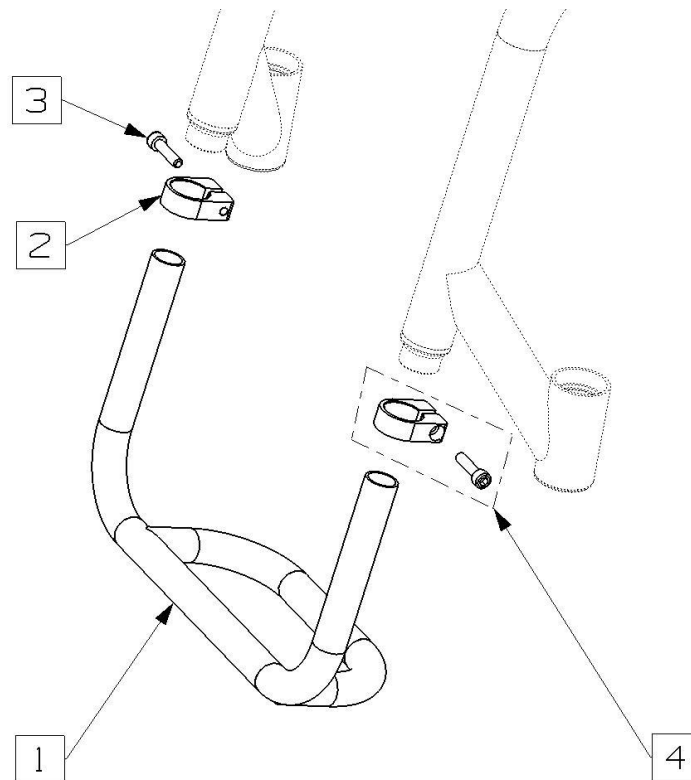


No.	Cat. No.	Element description	Qty
1	See part options below	Standard footrest	1
2	11-0522	Footrest clamp	2
3	SRU_DIN912_M6X25	Screw DIN912 M6x25	2
4	-	Footrest clamp with screw	(2)

	Part options – footrest tubes			
No.	Cat. No.	Element description		Qty
1 / A		Standard footrest for seat width 30 cm, black matt powder coated		1
1 / B		Standard footrest for seat width 32 cm, black matt powder coated		1
1 / C		Standard footrest for seat width 34 cm, black matt powder coated		1
1 / D		Standard footrest for seat width 36 cm, black matt powder coated		1
1 / E		Standard footrest for seat width 38 cm, black matt powder coated		1
1 / F		Standard footrest for seat width 40 cm, black matt powder coated		1
1 / G		Standard footrest for seat width 42 cm, black matt powder coated		1
1 / H		Standard footrest for seat width 44 cm, black matt powder coated		1
1 / I		Standard footrest for seat width 46 cm, black matt powder coated		1
1 / J		Standard footrest for seat width 48 cm, black matt powder coated		1
1 / K		Standard footrest for seat width 50 cm, black matt powder coated		1



Individual aluminum / titanium footrest

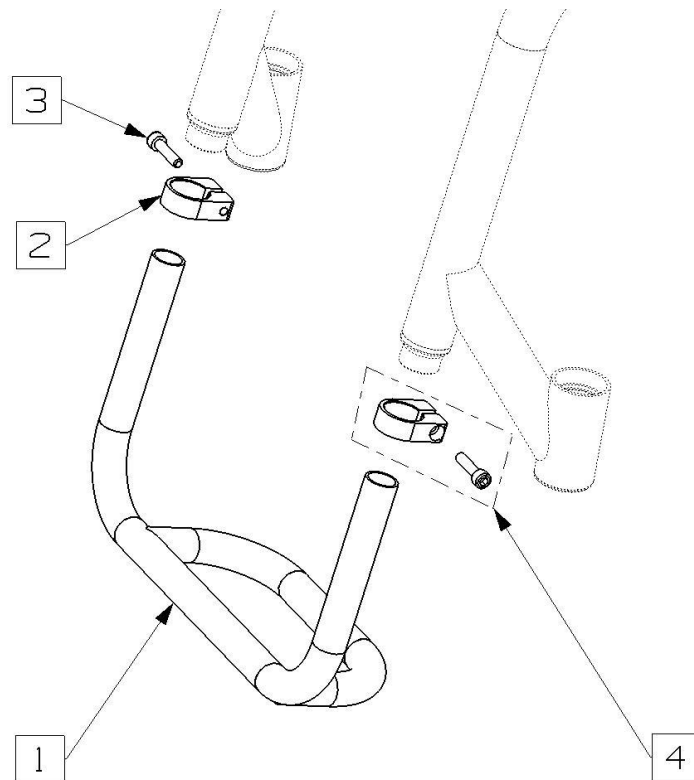


No.	Cat. No.	Element description	Qty
1	See part options below	Individual footrest	1
2	11-0522	Footrest clamp	2
3	SRU_DIN912_M6X25	Screw DIN912 M6x25	2
4	-	Footrest clamp with screw	(2)

Part options – footrest tubes

No.	Cat. No.	Element description	Qty
1		Individual footrest	1



Frame-closing bar

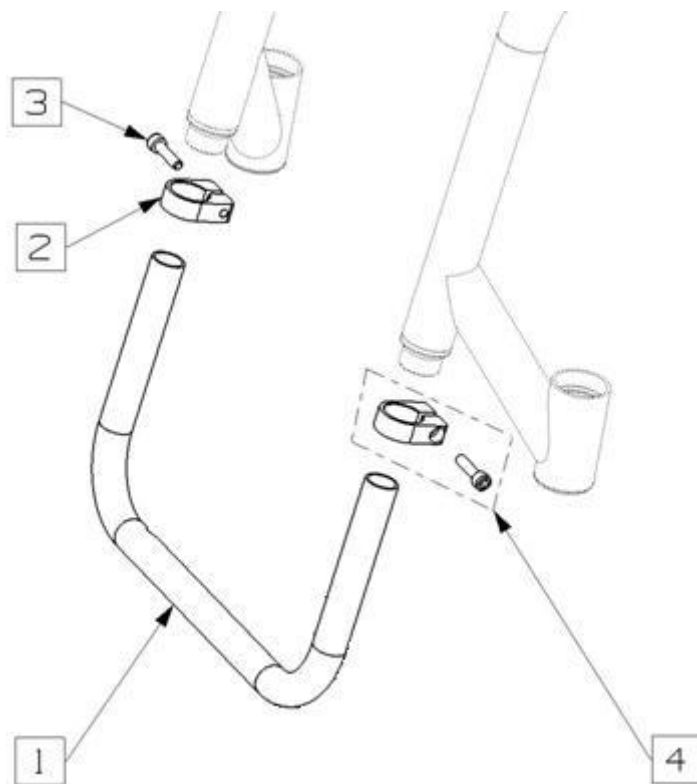


No.	Cat. No.	Element description	Qty
1	See part options below	Individual footrest	1
2	11-0522	Footrest clamp	2
3	SRU_DIN912_M6X25	Screw DIN912 M6x25	2
4	-	Footrest clamp with screw	(2)

Part options – footrest tubes

No.	Cat. No.	Element description	Qty
1		Frame-closing bar	1



Standard plastic footplatform

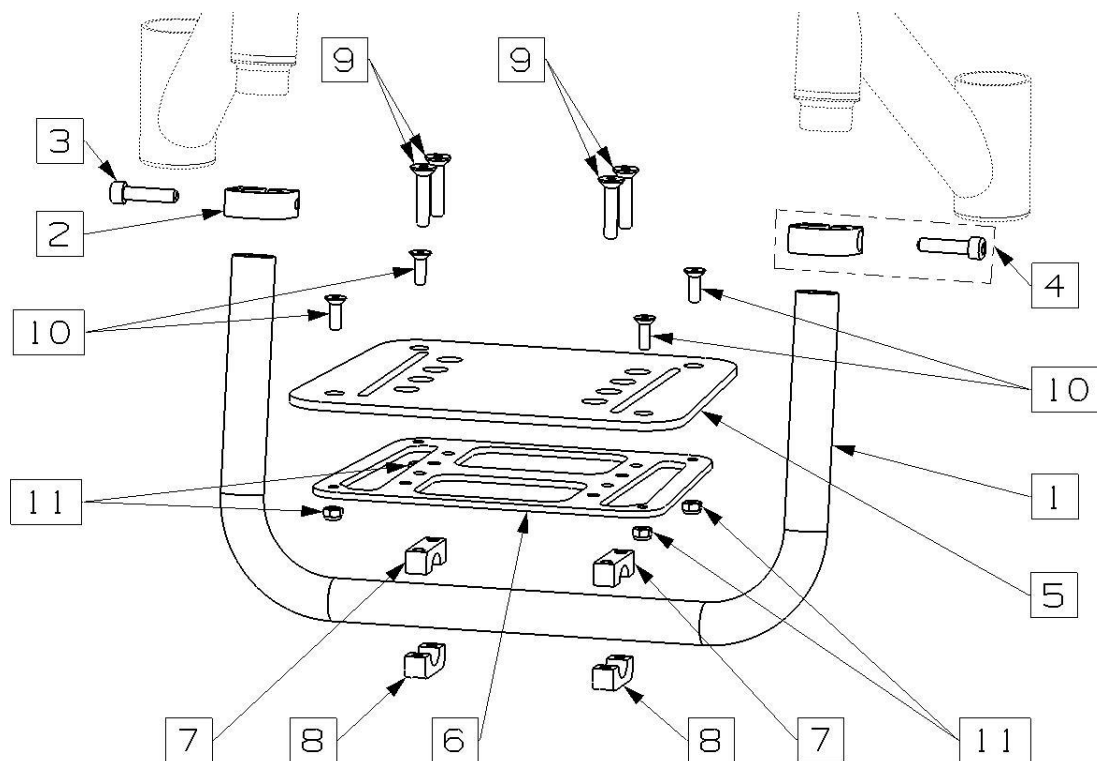


No.	Cat. No.	Element description	Qty
1	See part options below	Standard footplatform tube (Ø16mm or Ø20mm)	1
2	11-0522	Footrest clamp	2
3	SRU_DIN912_M6X25	Screw DIN912 M6x25	2
4	-	Footrest clamp with screw	(2)
5	PLAT_G1_ABS	Standard plastic plate	1
6	PLAT_G1_ALU	Standard aluminum reinforcement plate	1
7	11-0519 (Ø20) / 11-0518 (Ø16)	Upper clamp part	2
8	11-1097(Ø20) / 11-1096 (Ø16)	Lower clamp part	2
9	SRU_DIN7991_M6X35	Inbus screw M6 x 35	4
10	SRU_DIN7991_M5X10	Inbus screw M5 x 10	4
11	31-0267	Self-locking nut M5	4

Part options – footplatform tubes

No.	Cat. No.	Element description	Qty
1 / A	MG_02_SZ_30_PODNOZ	Standard footplatform tube for seat width 30 cm, black matt powder coated	1
1 / B	MG_02_SZ_32_PODNOZ	Standard footplatform tube for seat width 32 cm, black matt powder coated	1
1 / C	MG_02_SZ_34_PODNOZ	Standard footplatform tube for seat width 34 cm, black matt powder coated	1
1 / D	MG_02_SZ_36_PODNOZ	Standard footplatform tube for seat width 36 cm, black matt powder coated	1
1 / E	MG_02_SZ_38_PODNOZ	Standard footplatform tube for seat width 38 cm, black matt powder coated	1
1 / F	MG_02_SZ_40_PODNOZ	Standard footplatform tube for seat width 40 cm, black matt powder coated	1
1 / G	MG_02_SZ_42_PODNOZ	Standard footplatform tube for seat width 42 cm, black matt powder coated	1
1 / H	MG_02_SZ_44_PODNOZ	Standard footplatform tube for seat width 44 cm, black matt powder coated	1
1 / I	MG_02_SZ_46_PODNOZ	Standard footplatform tube for seat width 46 cm, black matt powder coated	1
1 / J	MG_02_SZ_48_PODNOZ	Standard footplatform tube for seat width 48 cm, black matt powder coated	1
1 / K	MG_02_SZ_50_PODNOZ	Standard footplatform tube for seat width 50 cm, black matt powder coated	1



Individual plastic or carbon fiber footplatform

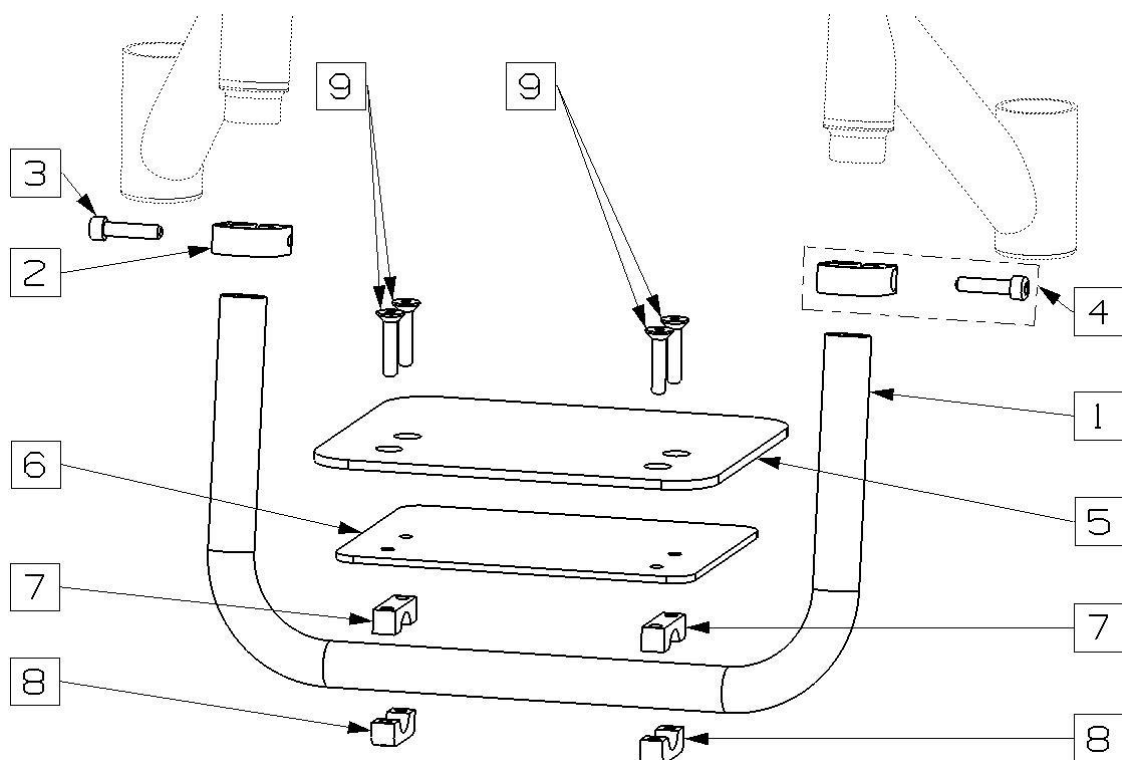


No.	Cat. No.	Element description	Qty
1	See part options below	Individual footplatform tube	1
2	11-0522	Footrest clamp	2
3	SRU_DIN912_M6X25	Screw DIN912 M6x25	2
4	-	Footrest clamp with screw	(2)
5	Individual part	Individual plastic or carbon fiberplate	1
6	Individual part	Individual aluminum reinforcement plate	1
7	11-0519 (Ø20) / 11-0518 (Ø16)	Upper clamp part	2
8	11-1097(Ø20) / 11-1096 (Ø16)	Lower clamp part	2
9	SRU_DIN7991_M6X35	Inbus screw M6 x 35	4

Part options – footplatform tubes


No.	Cat. No.	Element description	Qty
1		Individual footplatform tube	1



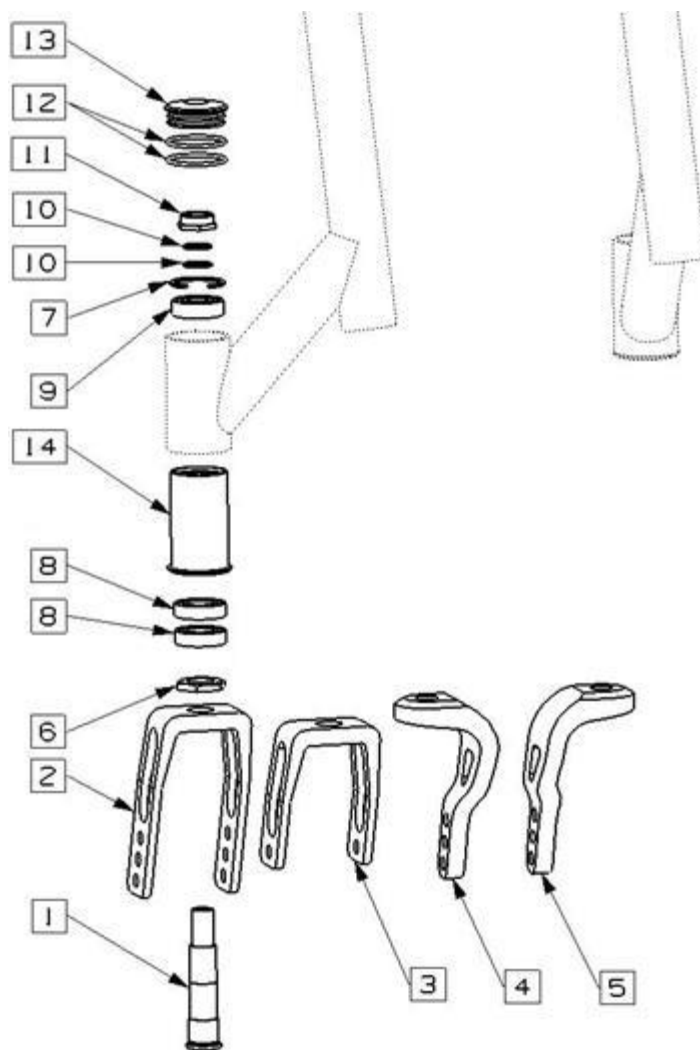

No.	Cat. No.	Element description	Qty
1	11-0599	Fork axle	2 X 1
2	WID_4_5_GW	Two-arm light fork, 3 holes	2 X 1
3	WID_4_GW	Two-arm extralight fork, 1 hole	2 X 1
4	11-0711	Mono-arm fork, right, 3 holes	1 X 1
5	11-0966	Mono-arm fork, left, 3 holes	1 X 1
6	NAKR_ALU_16X1_WOL	Nut M16x1	2 X 1
7	PODKL_DIN988_12_18_1	Retaining wing DIN472	2 X 3
8	LOZ_28-15-7_61902CX	Ball-bearing 61902	2 X 2
9	LOZ_28-12-8_6001NSK	Ball-bearing 6001	2 X 1
10	PODKL_DIN988_12_18_1	Steel spacer	2 X 2
11	31-0301	Self-locking nut M12	2 X 1
12	ORING_16X2_CX	O-ring	2 X 2
13	11-0110	Casters housing cap	2 X 1
14	11-0832	Fork sleeve	2 X 1


Plastic castor 4" x 26 mm for two-arm fork

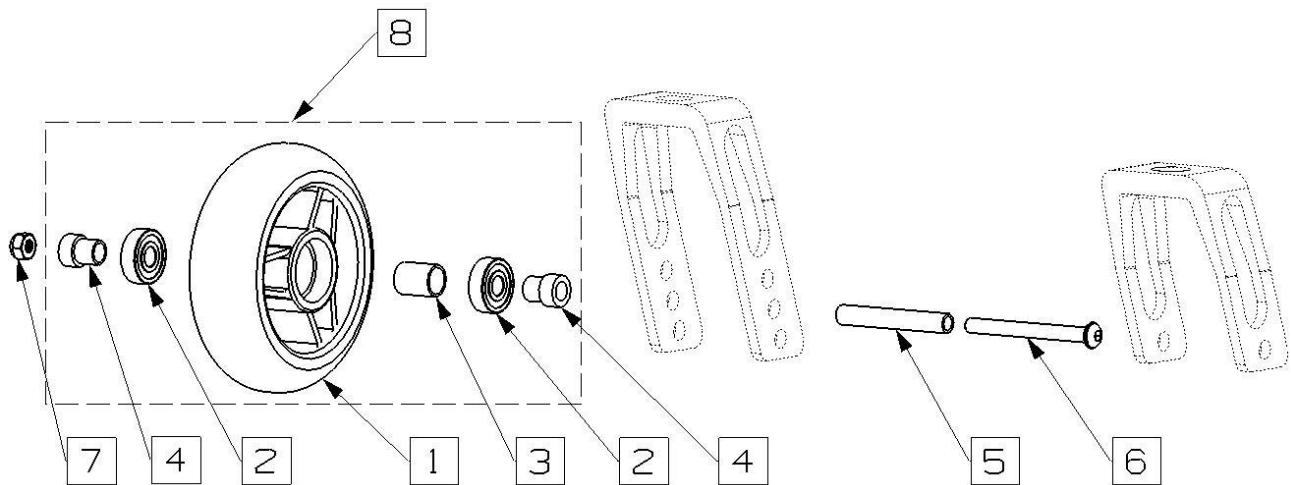

No.	Cat. No.	Element description	Qty
1	-	4" x 26mm plastic rim with tyre	2 x 1
2	LOZ_26-10-8_6000CX	Ball-bearing	2 x 2
3	-	Distance sleeve for bearings	2 x 1
4	-	Distance sleeve	2 x 1
5	11-0583	Steel axle	2 x 2
6	SRU_ISO7380_M6x60	Screw ISO7380 M6x60	2 x 2
7	NAKR_SAMOHAM_M6_A2	Self-locking nut M6	2 x 2
8	K_426_P_45	4" x 26mm plastic castor set	2 x 1




Plastic castor 4" x 32 mm for two-arm fork

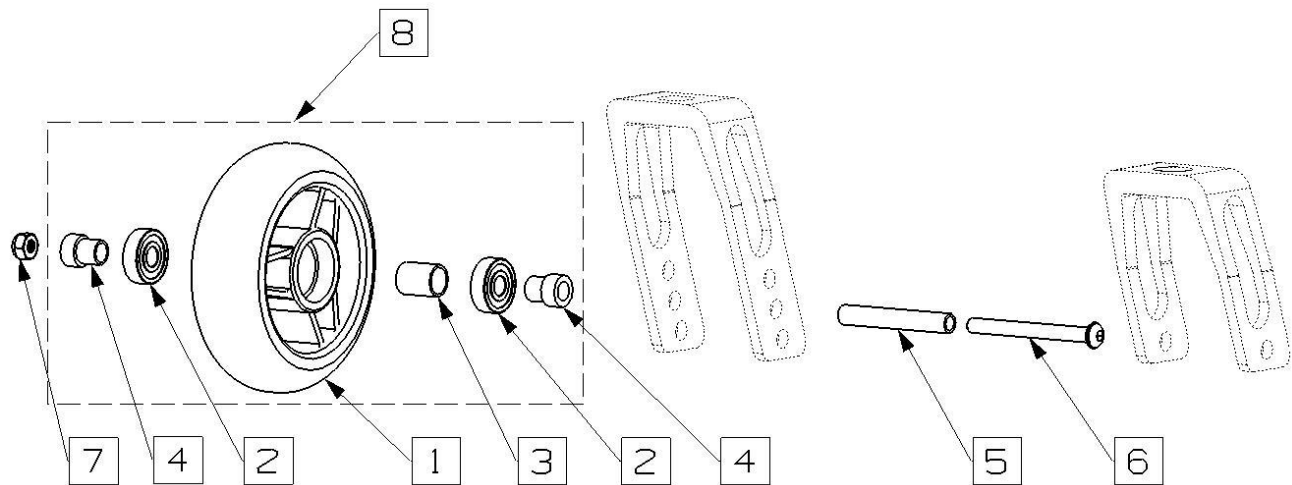

No.	Cat. No.	Element description	Qty
1	-	4" x 32mm plastic rim with tyre	2 X 1
2	LOZ_26-10-8_6000CX	Ball-bearing	2 X 2
3	-	Distance sleeve for bearings	2 X 1
4	-	Distance sleeve	2 X 1
5	11-0583	Steel axle	2 X 2
6	SRU_ISO7380_M6X60	Screw ISO7380 M6X60	2 X 2
7	NAKR_SAMOHAM_M6_A2	Self-locking nut M6	2 X 2
8	K_432_P	4" x 32mm plastic castor set	2 X 1

Aluminum caster 4" x 35 mm for two-arm fork

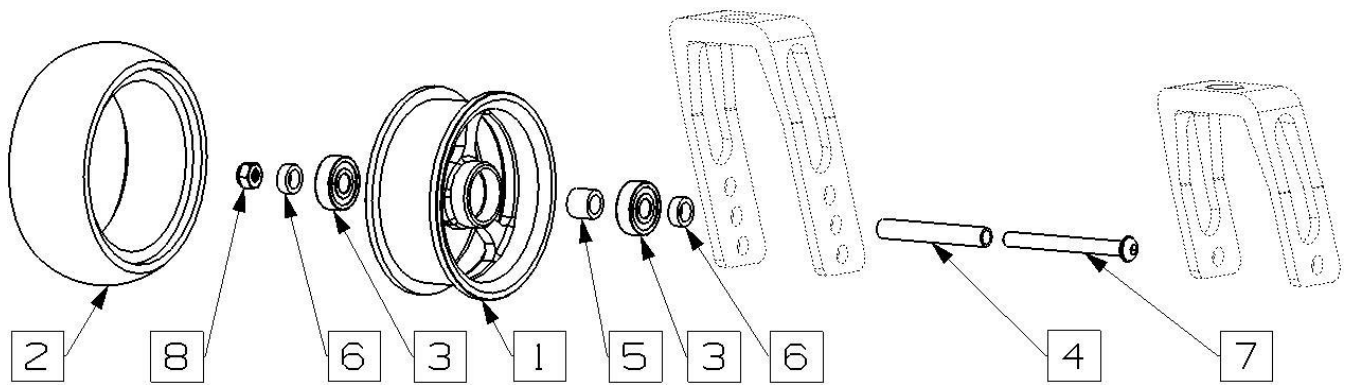


No.	Cat. No.	Element description	Qty
1	11-0851	4" x 35mm aluminum rim	2 X 1
2	SRT_4	Solid rubber tire	2 X 1
3	LOZ_22-8-7_608_NSK	Ball-bearing	2 X 2
4	11-0583	Steel axle	2 X 1
5	TUL_MOS_G21.3_F11/8	Distance sleeve for ball-bearings	2 X 1
6	11-0413	Distance sleeve	2 X 2
7	SRU_ISO7380_M6X60	Screw ISO7380 M6X60	2 X 2
8	NAKR_SAMOHAM_M6_A2	Self-locking nut M6	2 X 2

Set = 40 EUR




Aluminum castor 4" x 35 mm with sealed bearings for two-arm forks

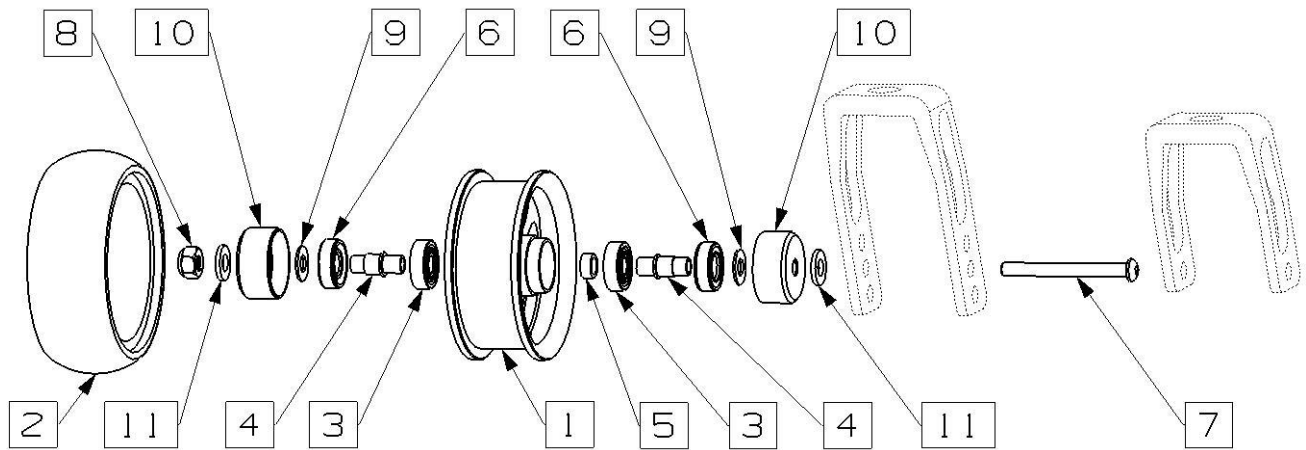

No.	Cat. No.	Element description	Qty
1	11-0799	Aluminum rim 4" x 35 mm	2 x 1
2	SRT4	Black Rubber 4" x 35 mm	2 x 1
3	LOZ_22-8-7_608_NSK	Ball-bearing	2 x 2
4	11-0586	Steel driveshaft	2 x 2
5	11-0479	Steel sleeve	2 x 1
6	USZCZ_10x22x7_CX	Simmering seal	2 x 2
7	SRU_ISO7380_M6X60	Screw ISO7380 M6 x 60 INOX	2 x 1
8	NARK_SAMOHAM_M6_A2	Self-locking nut M6	2 x 1
9	PODK_DIN988_8_14_10	Steel washer	2 x 2
10	11-0117	Aluminum cover	2 x 2
11	11-0202	Aluminum washer	2 x 2




Plastic castor 4" x 26 mm for monoarm fork

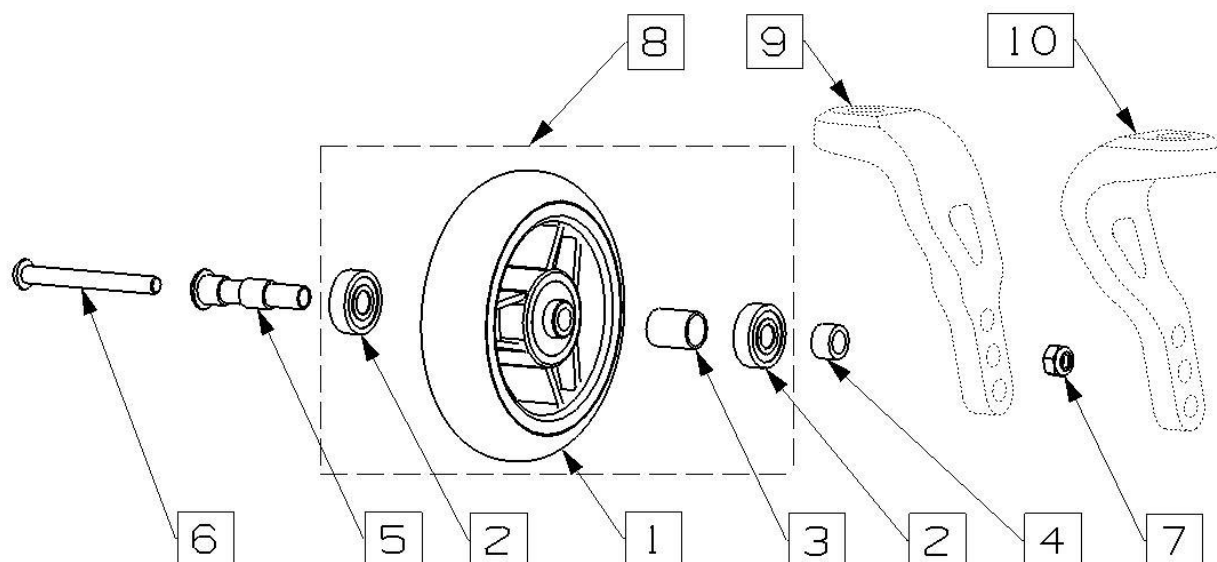

No.	Cat. No.	Element description	Qty
1	-	4" x 26mm plastic rim with tyre	2 x 1
2	LOZ_26-10-8_6000CX	Ball-bearing	2 x 2
3	-	Distance sleeve for bearings	2 x 1
4	-	Distance sleeve	2 x 1
5	11-0589	Steel axle	2 x 2
6	SRU_ISO7380_M6x50	Screw ISO7380 M6x50	2 x 2
7	NAKR_SAMOHAM_M6_A2	Self-locking nut M6	2 x 1
8	K_426_P_36	4" x 26mm plastic castor set	2 x 1
9	11-0711	Monoarm fork right	1
10	11-0966	Monoarm fork left	1




Plastic castor 4" x 32 mm for monoarm fork

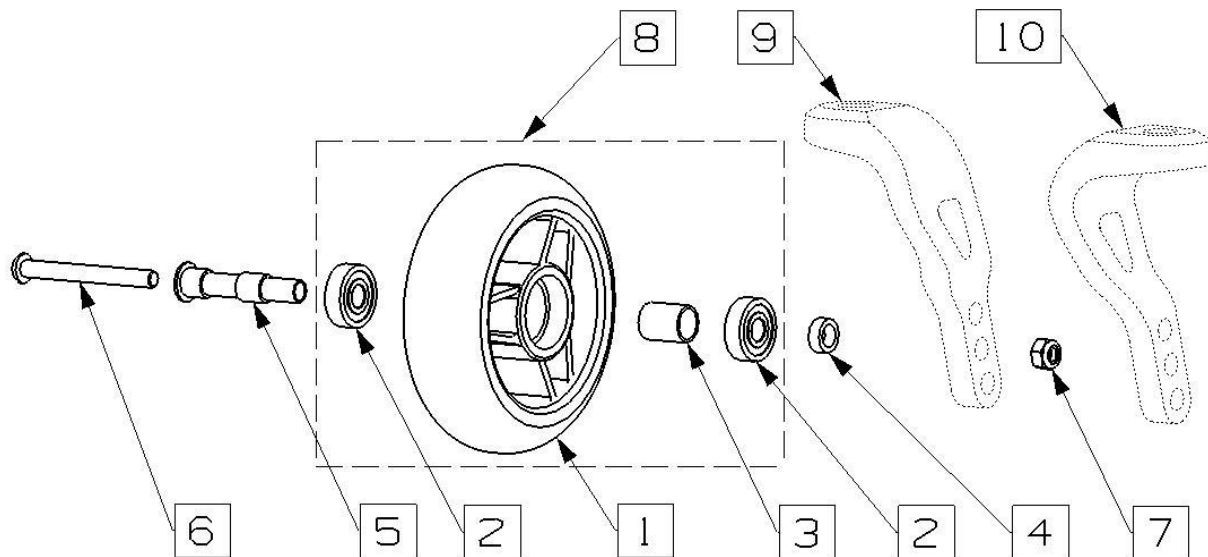

No.	Cat. No.	Element description	Qty
1	-	4" x 32mm plastic rim with tyre	2 x 1
2	LOZ_26-10-8_6000CX	Ball-bearing	2 x 2
3	-	Distance sleeve for bearings	2 x 1
4	-	Distance sleeve	2 x 1
5	11-0589	Steel axle	2 x 2
6	SRU_ISO7380_M6x55	Screw ISO7380 M6x55	2 x 2
7	NAKR_SAMOHAM_M6_A2	Self-locking nut M6	2 x 2
8	K_432_P_45_CZ	4" x 32mm plastic castor set	2 x 1
9	11-0711	Monoarm fork right	1
10	11-0966	Monoarm fork left	1




Aluminum castor 4" x 35 mm for monoarm fork

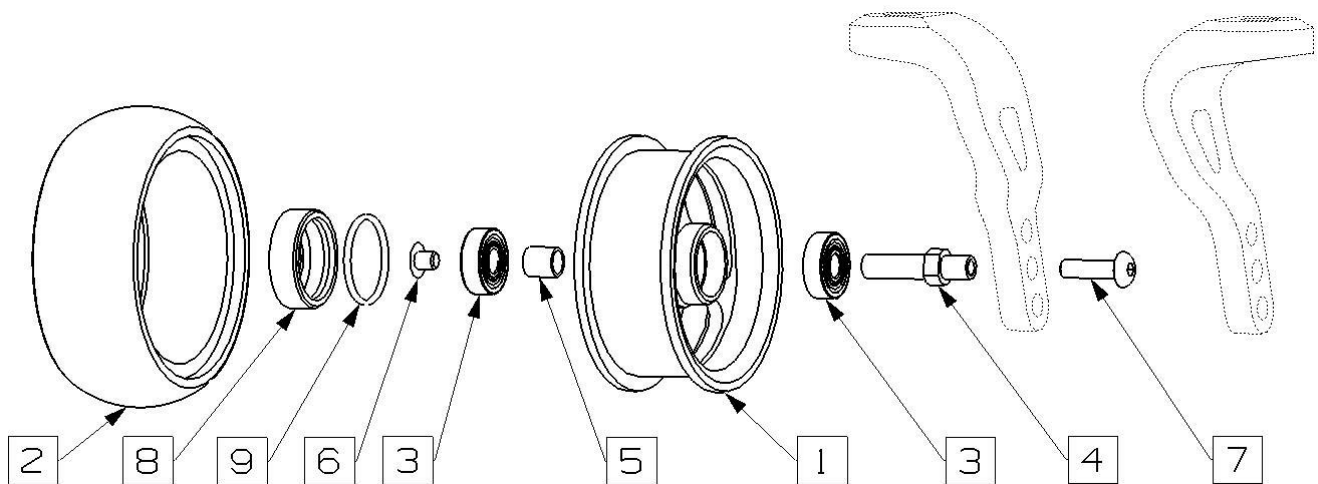

No.	Cat. No.	Element description	Qty
1	11-0857	4" x 35mm aluminum rim	2 x 1
2	SRT4	Black 4" x 35mm rubber	2 x 1
3	LOZ_22-8-7_608_NSK	Ball-bearing	2 x 2
4	11-0585	Steel axle	2 x 1
5	11-0458	Distance sleeve	2 x 1
6	SRU_ISO7380_M6x10	Screw ISO7380 M6x10	2 x 1
7	SRU_ISO7380_M6x25	Screw ISO7380 M6x25	2 x 1
8	11-0116	Aluminum sealing cap	2 x 1
9	O-RING_27x2_CX	O-ring sealing	2 x 1




Aluminum castor 4" x 35 mm with sealed bearings for monoarm forks

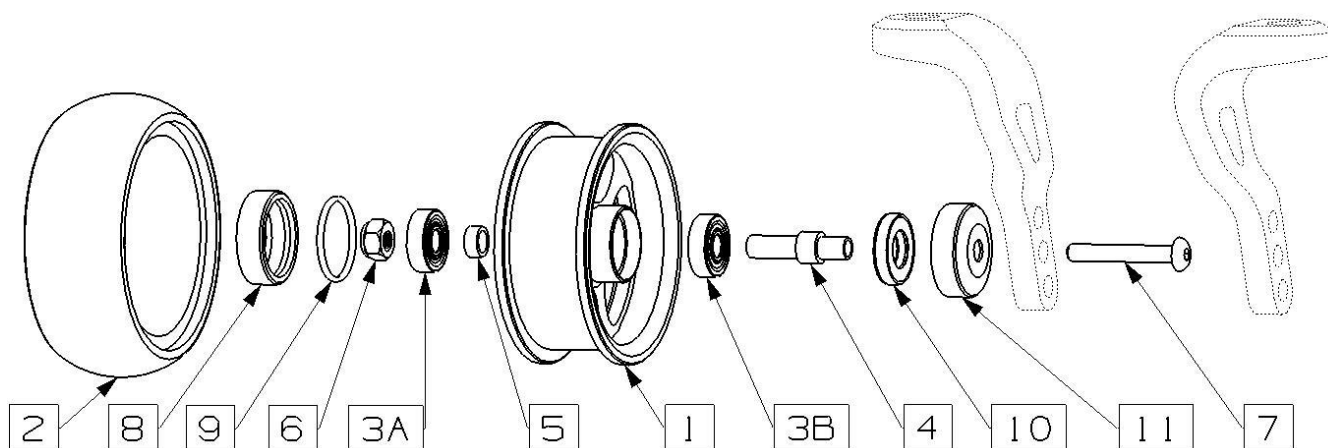

No.	Cat. No.	Element description	Qty
1	11-0856	Aluminum rim 4" x 35 mm for sealed bearings and monoarm fork	2 x 1
2	SRT4	Black Rubber 4" x 35 mm	2 x 1
3A	LOZ_22-8-7_608_NSK	External ball-bearing	2 x 1
3B	LOZ_22-8-7_608_NSK	Internal ball-bearing	2 x 1
4	11-0588	Steel driveshaft	2 x 1
5	11-0479	Steel sleeve	2 x 1
6	NAKR_SAMOHAM_M6_A2	Self-locking nut M6	2 x 2
7	SRU_ISO7380_M6x45	Screw ISO7380 M6 x 45 INOX	2 x 1
8	11-0120	External aluminum cover	2 x 1
9	ORING_27X2	O-ring	2 x 2
10	USZCZ_12x25x5	Simmering seal	2 x 2
11	11-0119	Internal aluminum cover	2 x 2




Plastic castor 5" x 29 mm for two-arm fork

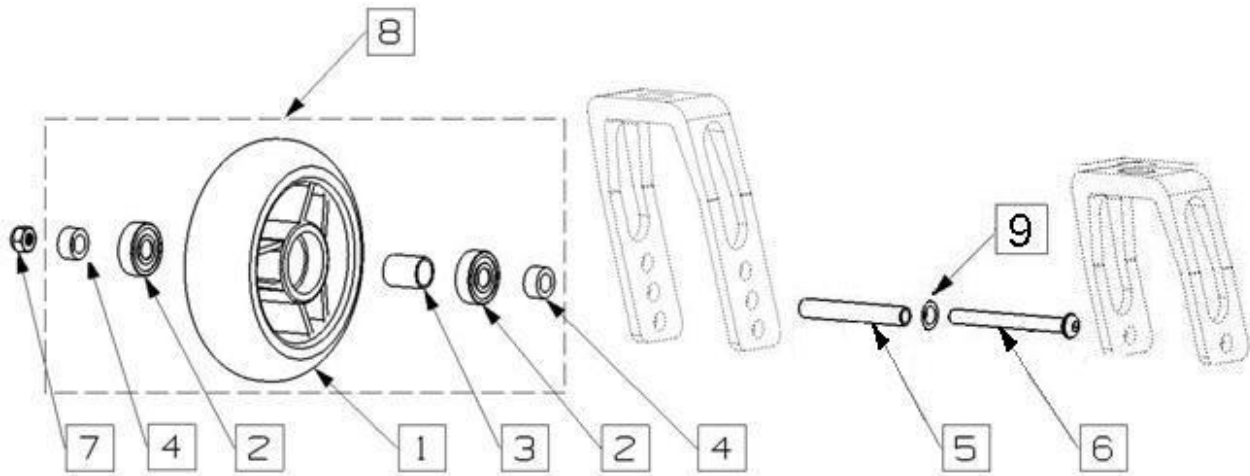

No.	Cat. No.	Element description	Qty
1	-	5" x 29mm plastic rim with tyre	2 X 1
2	LOZ_26-10-8_6000CX	Ball-bearing	2 X 2
3	-	Distance sleeve for bearings	2 X 1
4	11-0413	Distance sleeve	2 X 2
5	11-0583	Steel axle	2 X 1
6	SRU_ISO7380_M6X60	Screw ISO7380 M6X60	2 X 1
7	NAKR_SAMHAM_M6_A2	Self-locking nut M6	2 X 1
8	K_529_P_45	5" x 29mm plastic castor set	2 X 1
9	PODKL_DIN125_A2_M6	Washer	2 X 1




Plastic castor 5" x 27 mm for two-arm fork

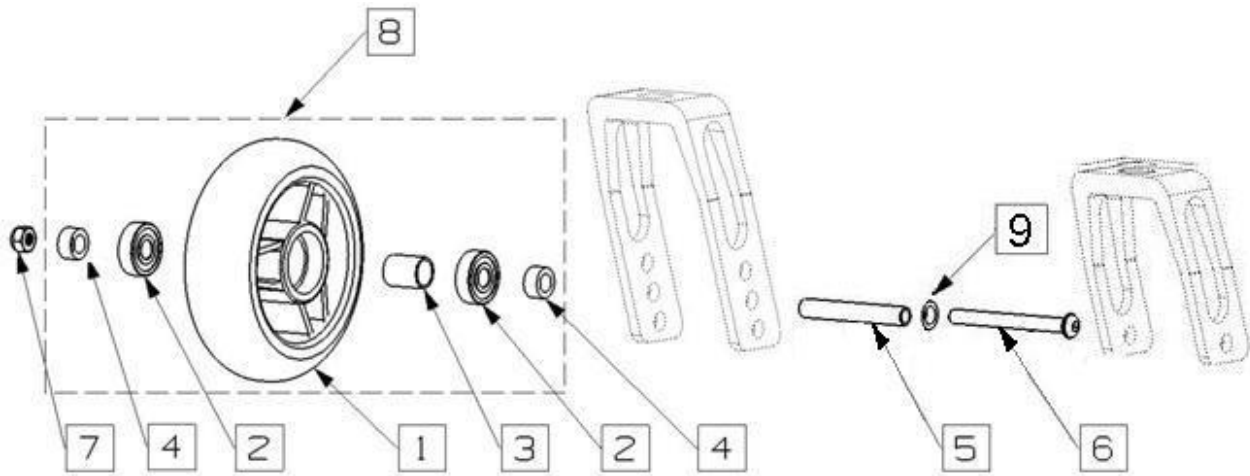

No.	Cat. No.	Element description	Qty
1	-	5" x 27mm plastic rim with tyre	2 X 1
2	LOZ_26-10-8_6000CX	Ball-bearing	2 X 2
3	-	Distance sleeve for bearings	2 X 1
4	11-0413	Distance sleeve	2 X 2
5	11-0583	Steel axle	2 X 1
6	SRU_ISO7380_M6X60	Screw ISO7380 M6X60	2 X 1
7	NAKR_SAMHAM_M6_A2	Self-locking nut M6	2 X 1
8	K_527_P	5" x 27mm plastic castor set	2 X 1




Aluminium castor 5" x 27 mm for two-arm fork

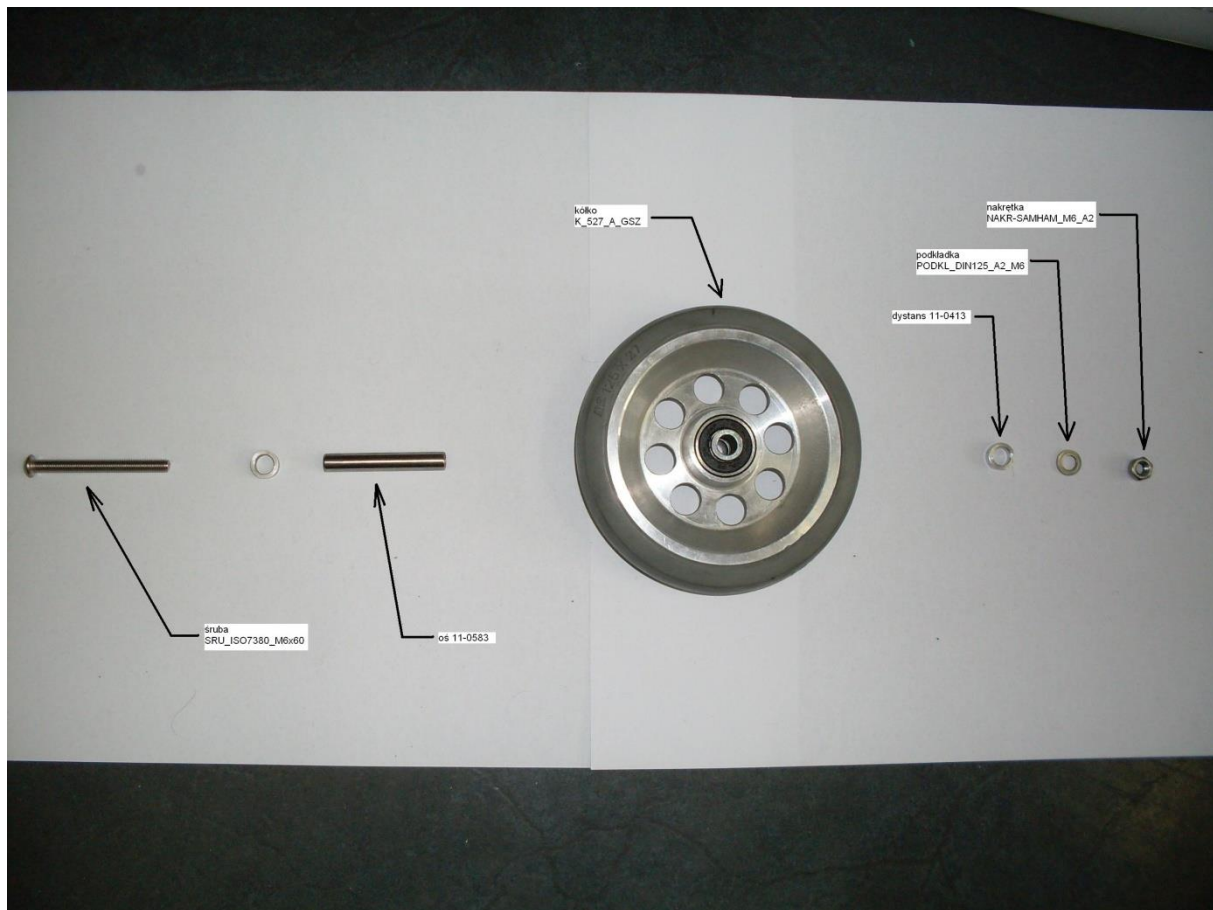

No.	Cat. No.	Element description	Qty
1	-	5" x 27mm aluminium rim with tyre	2 X 1
2	LOZ_26-10-8_6000CX	Ball-bearing	2 X 2
3	-	Distance sleeve for bearings	2 X 1
4	11-0413	Distance sleeve	2 X 2
5	11-0583	Steel axle	2 X 1
6	SRU_ISO7380_M6X60	Screw ISO7380 M6X60	2 X 1
7	NAKR_SAMHAM_M6_A2	Self-locking nut M6	2 X 1
8	K_527_A_GSZ	5" x 27mm aluminium castor set	2 X 1
9	PODKL_DIN125_A2_M6	Washer	2 X 1




Aluminum caster 5" x 35 mm for two-arm fork

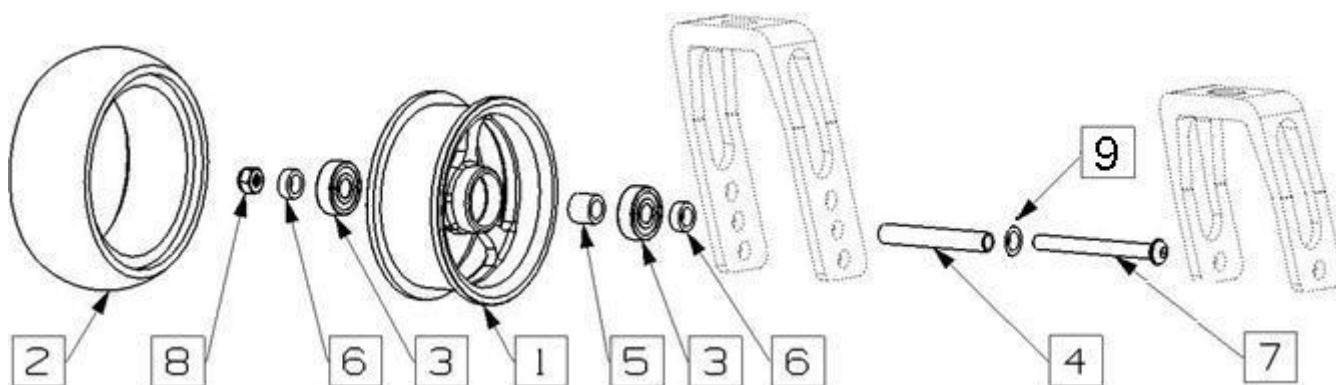

No.	Cat. No.	Element description	Qty
1	11-0804	5" x 35mm aluminum rim	2 X 1
2	SRT5	Solid rubber tire	2 X 1
3	LOZ_22-8-7_608_NSK	Ball-bearing	2 X 2
4	11-0583	Steel axle	2 X 1
5	TUL_MOS_G21.3F11/8	Distance sleeve for ball-bearings	2 X 1
6	11-0413	Distance sleeve	2 X 2
7	SRU_ISO7380_M6X60	Screw ISO7380 M6X60	2 X 1
8	NAKR_SAMHAM_M6_A2	Self-locking nut M6	2 X 1
9	PODKL_DIN125_A2_M6	Steel washer	2 X 1




Aluminum castor 5" x 35 mm with sealed bearings for two-arm forks

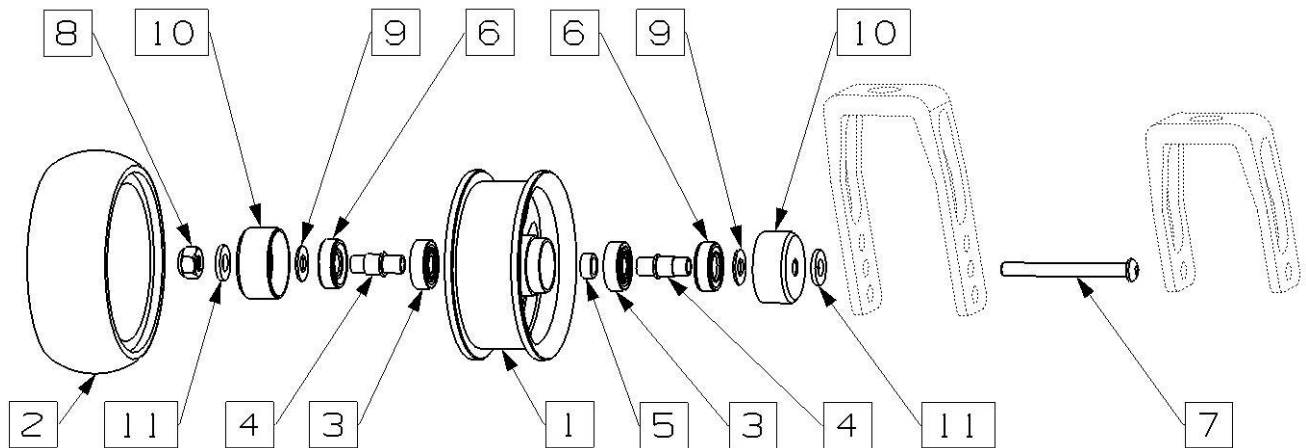

No.	Cat. No.	Element description	Qty
1	11-0803	Aluminum rim 5" x 35 mm	2 x 1
2	SRT5	Black Rubber 5" x 35 mm	2 x 1
3	11-0803	Ball-bearing	2 x 2
4	11-0586	Steel driveshaft	2 x 2
5	TUL_STAL_G4,8_F11/8	Distance sleeve for bearings	2 x 1
6	USZCZ-10X22X7_CX	Simmering seal	2 x 2
7	SRU_ISO7380_M6X60	Screw ISO7380 M6X60	2 x 1
8	NAKR_SAMHAM_M6_A2	Self-locking nut M6	2 x 1
9	POK_DIN988_8-7_NSK	Washer	2 x 2
10	11-0118	Aluminum cover	2 x 2
11	PODKL_DIN125_A2_M6	Washer	2 x 2




Plastic castor 5" x 29 mm for monoarm fork

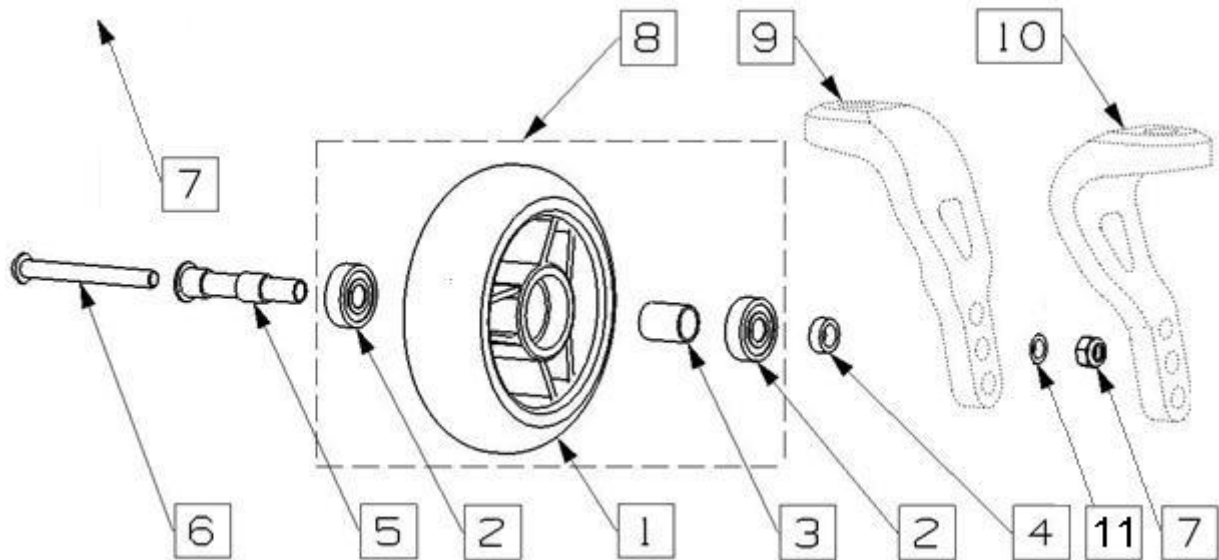

No.	Cat. No.	Element description	Qty
1	-	5" x 29mm plastic rim with tyre	2 x 1
2	LOZ_26-10-8_6000CX	Ball-bearing	2 x 2
3	-	Distance sleeve for bearings	2 x 1
4	11-0413	Distance sleeve	2 x 1
5	11-0589	Steel axle	2 x 1
6	SRU_ISO7380_M6x55	Screw ISO7380 M6x55	2 x 1
7	NAKR_SAMHAM_M6_A2	Self-locking nut M6	2 x 1
8	K_529_P_45	5" x 29mm plastic castor set	2 x 1
9	11-0711	Monoarm fork right	1
10	11-0966	Monoarm fork left	1
11	PODKL_DIN125_A2_M6	Washer	2 x 1




Aluminum castor 5" x 35 mm for monoarm fork

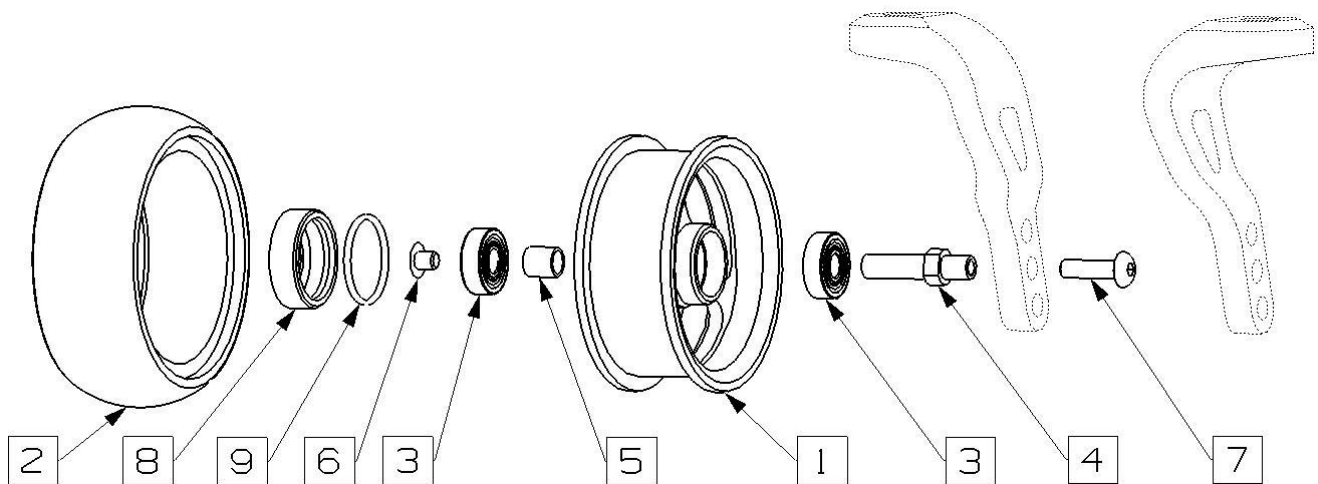

No.	Cat. No.	Element description	Qty
1	11-0855	5" x 35mm aluminum rim	2 x 1
2	SRT5	Black 5" x 35mm rubber	2 x 1
3	LOZ_22-8-7_608_NSK	Ball-bearing	2 x 2
4	OS_K_A1	Steel axle	2 x 1
5	TUL_MOS_L10,5_F10/8	Distance sleeve for bearings	2 x 1
6	SRU_ISO7380_M6x10	Screw ISO7380 M6x10	2 x 1
7	SRU_ISO7380_M6x25	Screw ISO7380 M6x25	2 x 1
8	KAPSEL_K_A1	Aluminum cover	2 x 1
9	O-RING_27X2_CX	O-ring sealing	2 x 1




Aluminum castor 5" x 35 mm with sealed bearings for monoarm fork

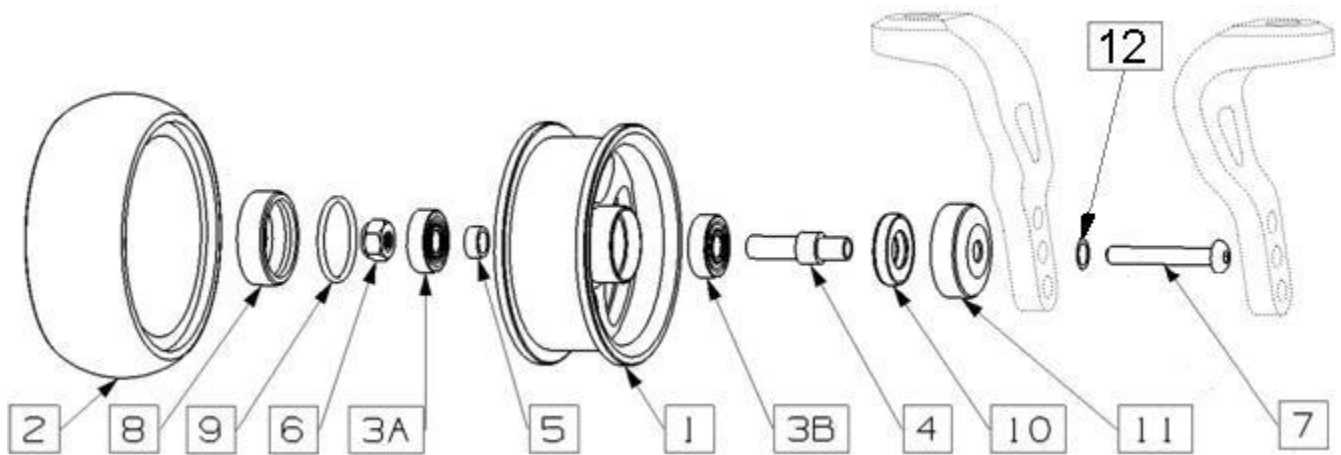

No.	Cat. No.	Element description	Qty
1	11-0858	Aluminum rim 5" x 35 mm for sealed bearings and monoarm fork	2 x 1
2	SRT5	Black Rubber 5" x 35 mm	2 x 1
3	LOZ_22-8-7_608_NSK	Ball-bearing	2 x 2
4	11-0588	Steel driveshaft	2 x 1
5	TUL_STAL_G4,8_F11/8	Distance sleeve for bearings	2 x 1
6	NAKR_SAMHAM_M6_A2	Self-locking nut M6	2 x 1
7	SRU_7380_M6X45	Screw ISO7380 M6 x 45	2 x 1
8	11-0120	External aluminum cover	2 x 1
9	O-RING_27X2_CX	O-ring	2 x 1
10	USZCZ_12X25X5	Simmering seal	2 x 1
11	11-0119	Internal aluminum cover	2 x 1
12	PODKL_DIN125_A2_M6	Washer	2 x 1



Removable sideguard for 44 cm rear seat height

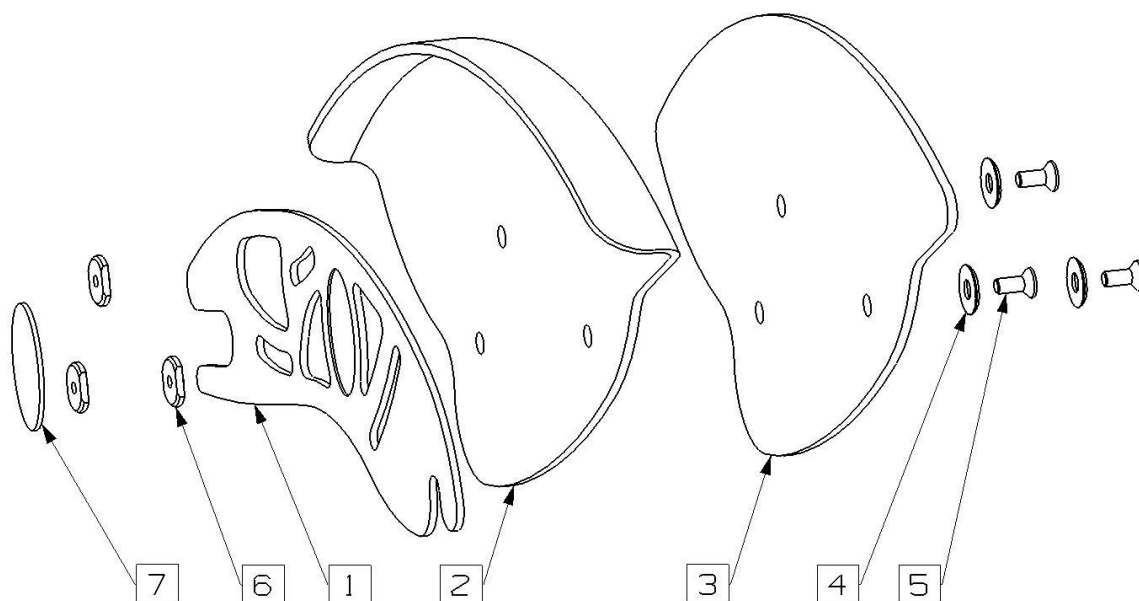

No.	Cat. No.	Element description	Qty
1	See options list below	Sideguard aluminium part, for 44 cm rear seat height	2 X 1
2	See options list below	Fender with mudguard	2 X (1)
3	See options list below	Simple fender	2 X (1)
4	PODKL_BOK_F20/9	Aluminium washer	2 X 3
5	SRU_DIN7991_M6X12	Screw M6 x 12	2 X 3
6	NAKR_BOK_M6	M6 aluminum nut	2 X 3
7	NAKLEJKA_ALU_GTM_D0	Aluminium sticker with GTM logo	2 X 1

Part options – sideguard aluminium part

No.	Cat. No.	Element description	Qty
1 / A	BOK_MG07_ZEB_WYC_ULP	Sideguard aluminium part, for 44 cm rear seat height, right	1
1 / B	BOK_MG07_ZEB_WYC_ULL	Sideguard aluminium part, for 44 cm rear seat height, left	1

Part options – fender with mudguard


No.	Cat. No.	Element description	Qty
2 / A	ABS_MG07_BL_P	Fender with mudguard, plastic ABS, right	(1)
2 / B	ABS_MG07_BL_L	Fender with mudguard, plastic ABS, left	(1)
2 / C	-	Fender with mudguard, carbon fibers, right	(1)
2 / D	-	Fender with mudguard, carbon fibers, left	(1)

Part options – Simple fender

No.	Cat. No.	Element description	Qty
3 / A	ABS_MG07_PR_P	Simple fender, plastic ABS, right	(1)
3 / B	ABS_MG07_PR_P	Simple fender, plastic ABS, left	(1)
3 / C	-	Simple fender, carbon fibers, right	(1)
3 / D	-	Simple fender, carbon fibers, left	(1)



Removable sideguard for 42 cm rear seat height

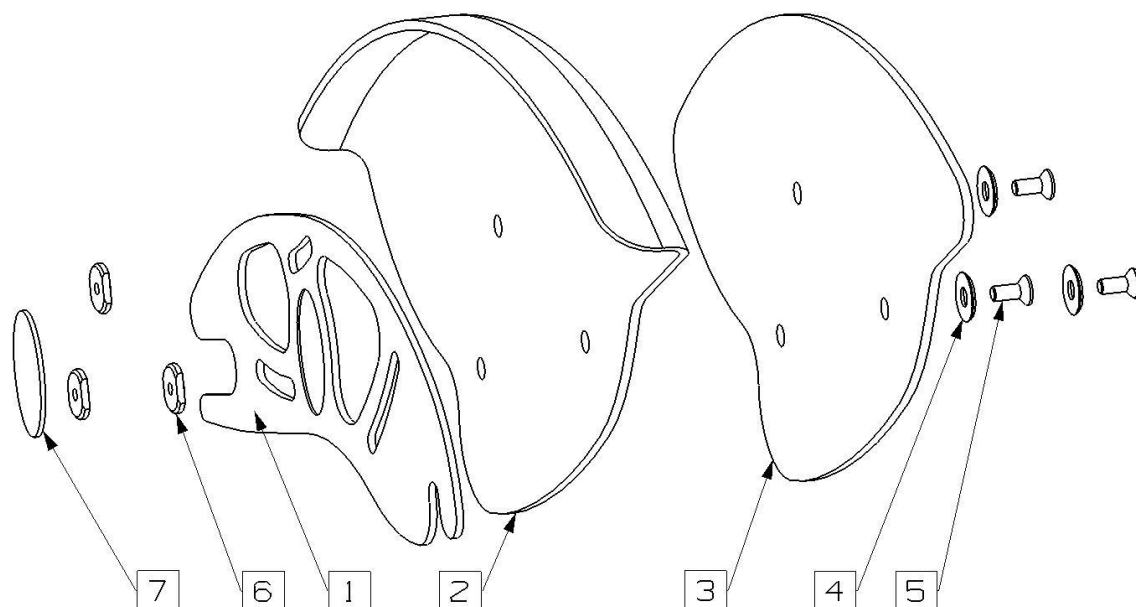

No.	Cat. No.	Element description	Qty
1	See options list below	Sideguard aluminium part, for 42 cm rear seat height	2 X 1
2	See options list below	Fender with mudguard	2 X (1)
3	See options list below	Simple fender	2 X (1)
4	PODKL_BOK_F20/9	Aluminium washer	2 X 3
5	SRU_DIN7991_M6X12	Screw M6 x 12	2 X 3
6	NAKR_BOK_M6	M6 aluminum nut	2 X 3
7	NAKLEJKA_ALU_GTM_D0	Aluminium sticker with GTM logo	2 X 1

Part options – sideguard aluminium part

No.	Cat. No.	Element description	Qty
1 / A	BOK_MG10_ZEB_WYC_ULP	Sideguard aluminium part, for 42 cm rear seat height, right	1
1 / B	BOK_MG10_ZEB_WYC_ULL	Sideguard aluminium part, for 42 cm rear seat height, left	1

Part options – fender with mudguard


No.	Cat. No.	Element description	Qty
2 / A	ABS_MG10_BL_P	Fender with mudguard, plastic ABS, right, for 42 cm rear seat height	(1)
2 / B	ABS_MG10_BL_L	Fender with mudguard, plastic ABS, left, for 42 cm rear seat height	(1)
2 / C	-	Fender with mudguard, carbon fibers, right, for 42 cm rear seat height	(1)
2 / D	-	Fender with mudguard, carbon fibers, left, for 42 cm rear seat height	(1)

Part options – simple fender

No.	Cat. No.	Element description	Qty
3 / A	ABS_MG10_PR_P	Simple fender, plastic ABS, right, for 42 cm rear seat height	(1)
3 / B	ABS_MG10_PR_P	Simple fender, plastic ABS, left, for 42 cm rear seat height	(1)
3 / C	-	Simple fender, carbon fibers, right, for 42 cm rear seat height	(1)
3 / D	-	Simple fender, carbon fibers, left, for 42 cm rear seat height	(1)



Removable sideguard for 40 cm rear seat height

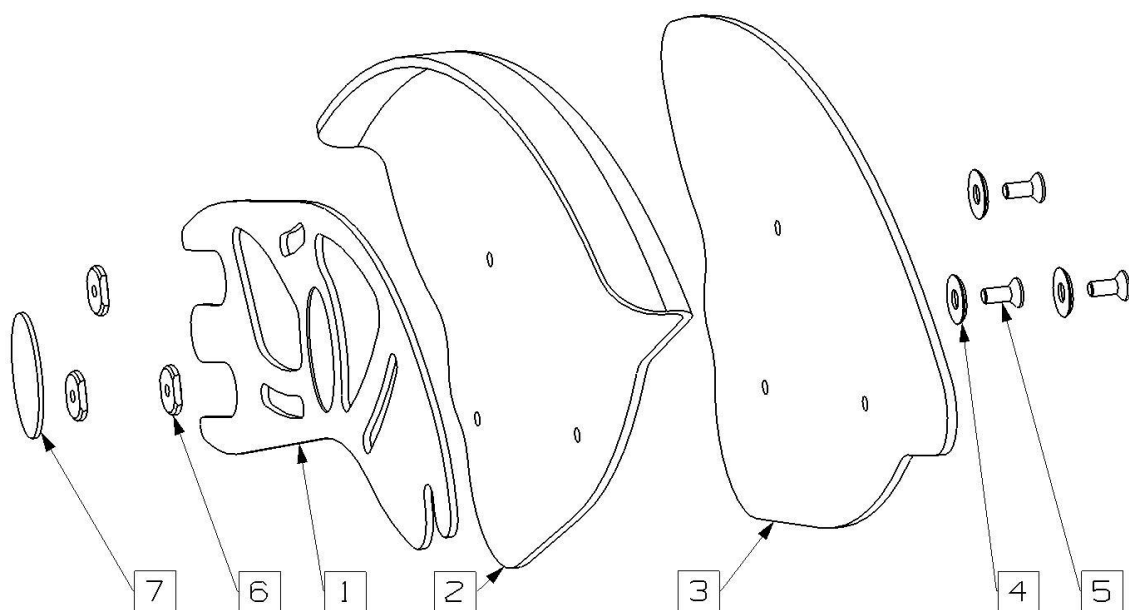

No.	Cat. No.	Element description	Qty
1	See options list below	Sideguard aluminium part, for 40 cm rear seat height	2 X 1
2	See options list below	Fender with mudguard	2 X (1)
3	See options list below	Simple fender	2 X (1)
4	PODKL_BOK_F20/9	Aluminium washer	2 X 3
5	SRU_DIN7991_M6X12	Screw M6 x 12	2 X 3
6	NAKR_BOK_M6	M6 aluminum nut	2 X 3
7	NAKLEJKA_ALU_GTM_D0	Aluminium sticker with GTM logo	2 X 1

Part options – sideguard aluminium part

No.	Cat. No.	Element description	Qty
1 / A	11-0075	Sideguard aluminium part, for 40 cm rear seat height, right	1
1 / B	11-0074	Sideguard aluminium part, for 40 cm rear seat height, left	1

Part options – fender with mudguard

No.	Cat. No.	Element description	Qty
2 / A	ABS_MG13_BL_P	Fender with mudguard, plastic ABS, right, for 40 cm rear seat height	(1)
2 / B	ABS_MG13_BL_L	Fender with mudguard, plastic ABS, left, for 40 cm rear seat height	(1)
2 / C	-	Fender with mudguard, carbon fibers, right, for 40 cm rear seat height	(1)
2 / D	-	Fender with mudguard, carbon fibers, left, for 40 cm rear seat height	(1)

Part options – Simple fender

No.	Cat. No.	Element description	Qty
3 / A	ABS_MG13_PR_P	Simple fender, plastic ABS, right, for 40 cm rear seat height	(1)
3 / B	ABS_MG13_PR_P	Simple fender, plastic ABS, left, for 40 cm rear seat height	(1)
3 / C	-	Simple fender, carbon fibers, right, for 40 cm rear seat height	(1)
3 / D	-	Simple fender, carbon fibers, left, for 40 cm rear seat height	(1)



Sideguard folding with the backrest

No.	Cat. No.	Element description	Qty
1	See options list below	Sideguard aluminium part, for 40 cm rear seat height	2 X 1
2	See options list below	Fender with mudguard	2 X (1)
3	See options list below	Simple fender	2 X (1)
4	PODKL_BOK_F20/9	Aluminium washer	2 X 3
5	SRU_DIN7991_M6X12	Screw M6 x 12	2 X 3
6	NAKR_BOK_M6	M6 aluminum nut	2 X 3
7	NAKLEJKA_ALU_GTM_D0	Aluminium sticker with GTM logo	2 X 1




Removable armrest

No.	Cat. No.	Element description	Qty
1	See options list below	Sideguard aluminium part, for 40 cm rear seat height	2 X 1
2	See options list below	Fender with mudguard	2 X (1)
3	See options list below	Simple fender	2 X (1)
4	PODKL_BOK_F20/9	Aluminium washer	2 X 3
5	SRU_DIN7991_M6X12	Screw M6 x 12	2 X 3
6	NAKR_BOK_M6	M6 aluminum nut	2 X 3
7	NAKLEJKA_ALU_GTM_D0	Aluminium sticker with GTM logo	2 X 1




Scissor brakes with clamps

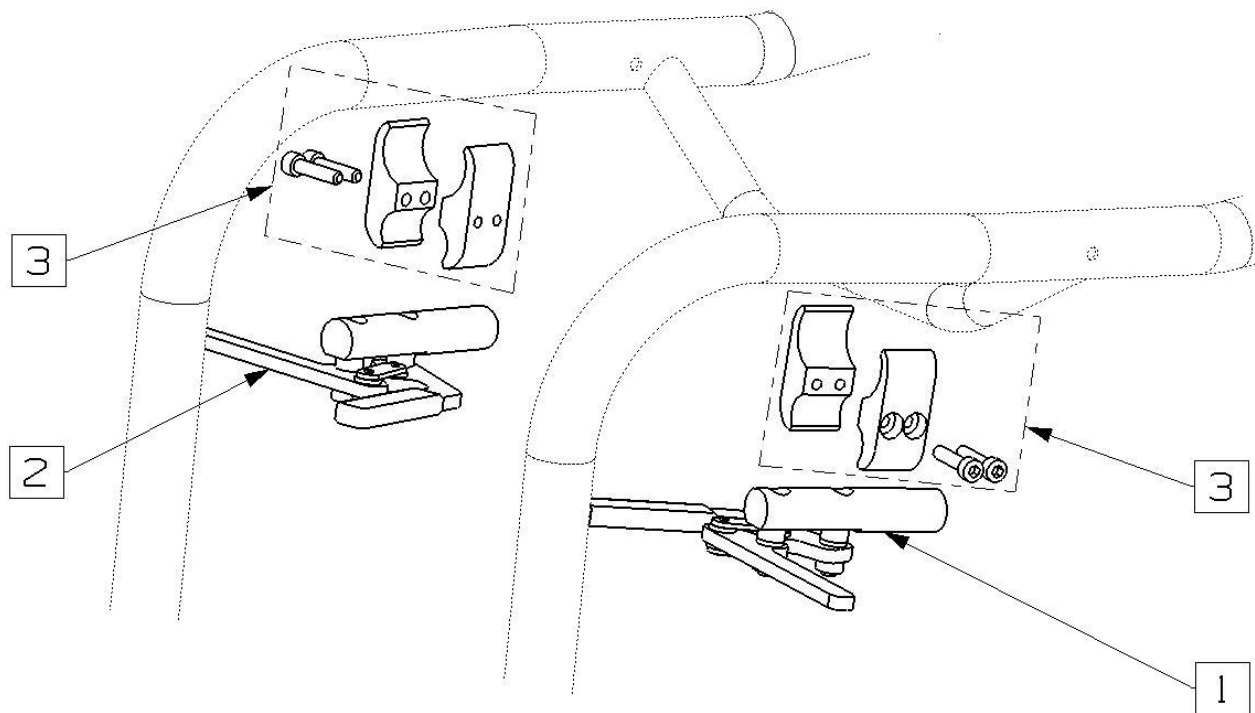

No.	Cat. No.	Element description		Qty
1	HAM_SPORT_L	Scissor brake, left		1
2	HAM_SPORT_P	Scissor brake, right		1
3	KLAMRA_HAM_19-30	Brake clamp		2




Side brakes with clamps

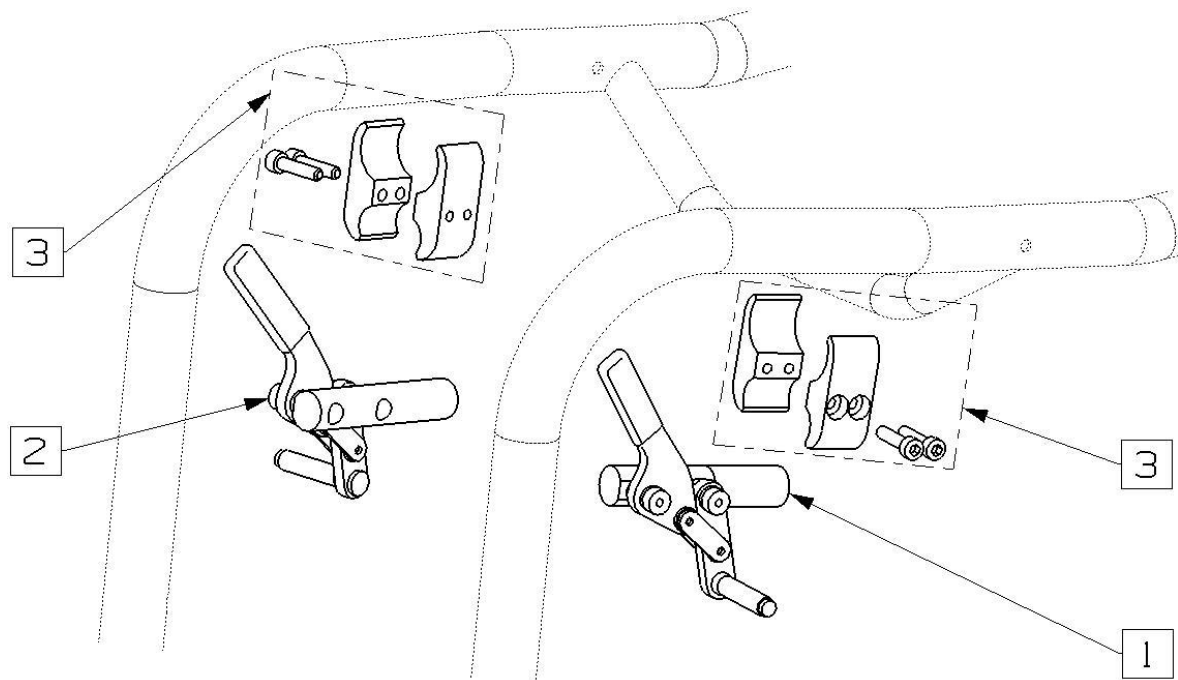

No.	Cat. No.	Element description	Qty
1	HAM_STD_LEWY_MBL	Side brake, left	1
2	HAM_STD_PRAWY_MBL	Side brake, right	1
3	KLAMRA_HAM_19-30	Brake clamp	2


Front mounting point for removable sideguard

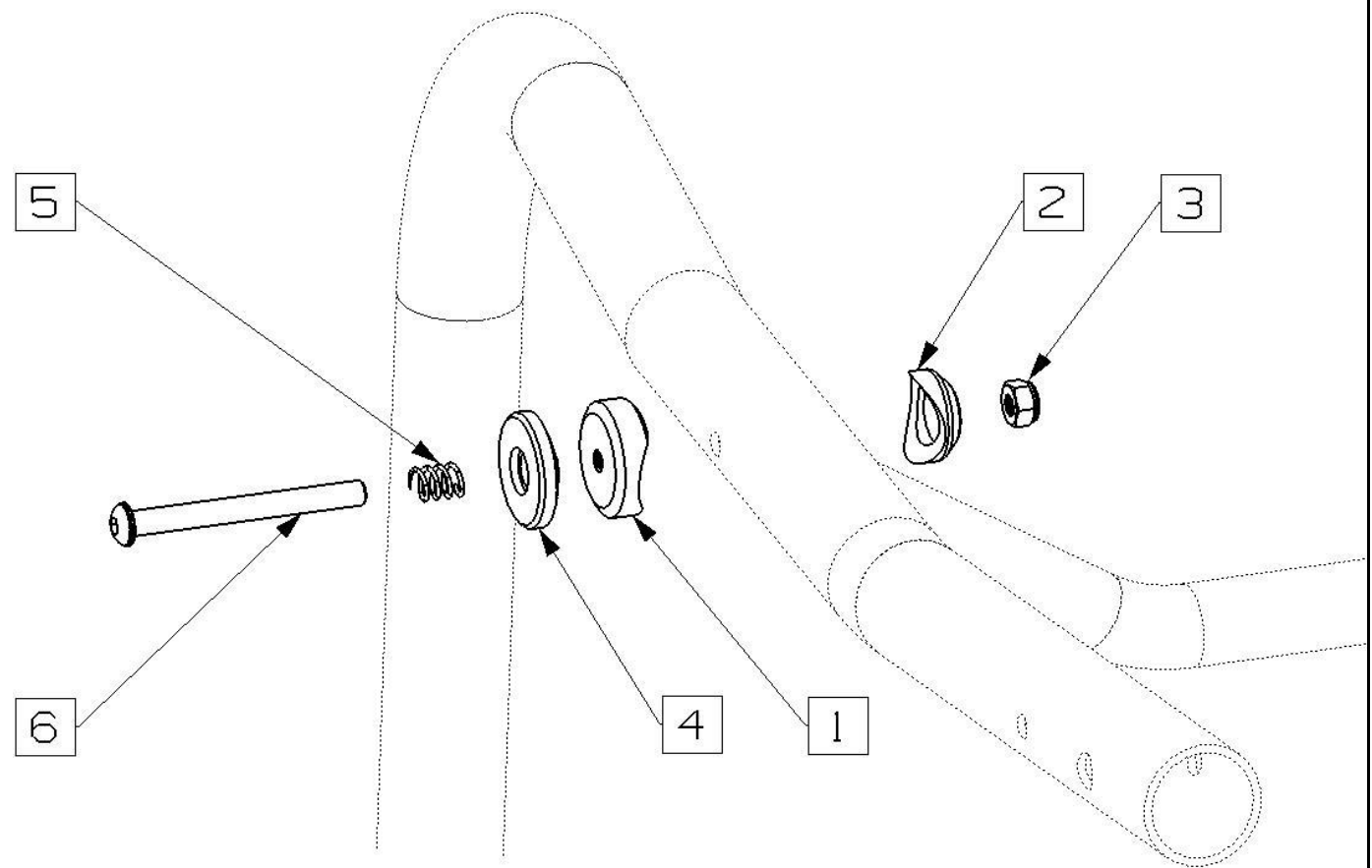

No.	Cat. No.	Element description	Qty
1	11-0308	Outer saddle washer	2 X 1
2	11-0309	Inner saddle washer	2 X 1
3	NAKR_SAMOHAM_M6_A2	Self-locking nut M6	2 X 1
4	11-0319	Clamping disc	2 X 1
5	SPREZ_BOK	Spring	2 X 1
6	SRU_ISO7380_M6X55	Bolt M6 x 55	2 X 1




Left mounting point of folding backrest mechanism

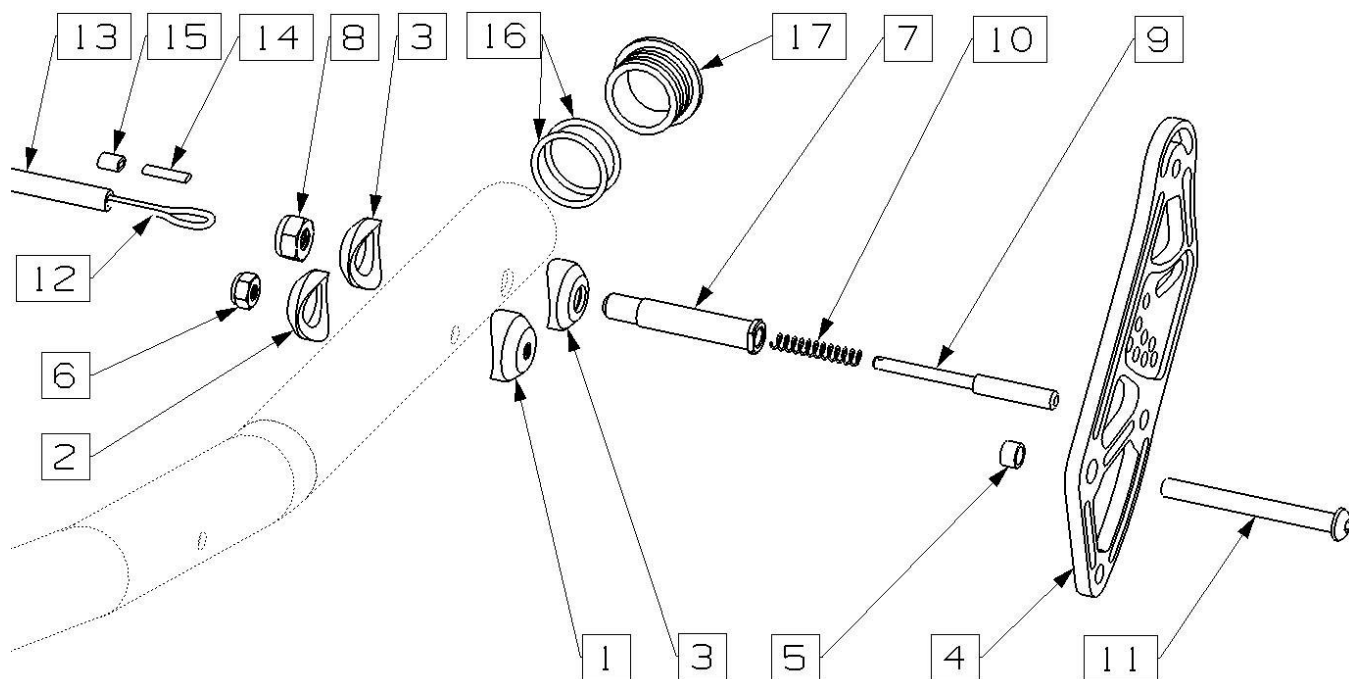

No.	Cat. No.	Element description	Qty
1	11-0310	Saddle washer	1
2	11-0309	Saddle washer	1
3	11-0307	Saddle washer	2
4	11-0545	Angle setting plate for left side	1
5	11-0405	Steel sleeve	1
6	NAKR_SAMPHAM_M6_CX	Self-locking nut M6	1
7	31-0576	Locking mechanism steel sleeve	1
8	NAKR_SAMOHAM_M8_CX	Self-locking nut M8	1
9	31-0580	Locking steel bolt	1
10	SPREZ_ZAMKA	Spring	1
11	SRU_ISO_7380_M6X55	Bolt M6 x 55	1
12	-	Steel link	1
13	-	Plastic transparent link integument	1
14	73-0071	Plastic link cover	2
15	ZACISK_LINKI	Metal clips	2
16	ORING_21X2	O-ring	2
17	KAP_RAM_FI30_GRAW_L	Rear frame aluminum cap, left	1




Right mounting point of folding backrest mechanism

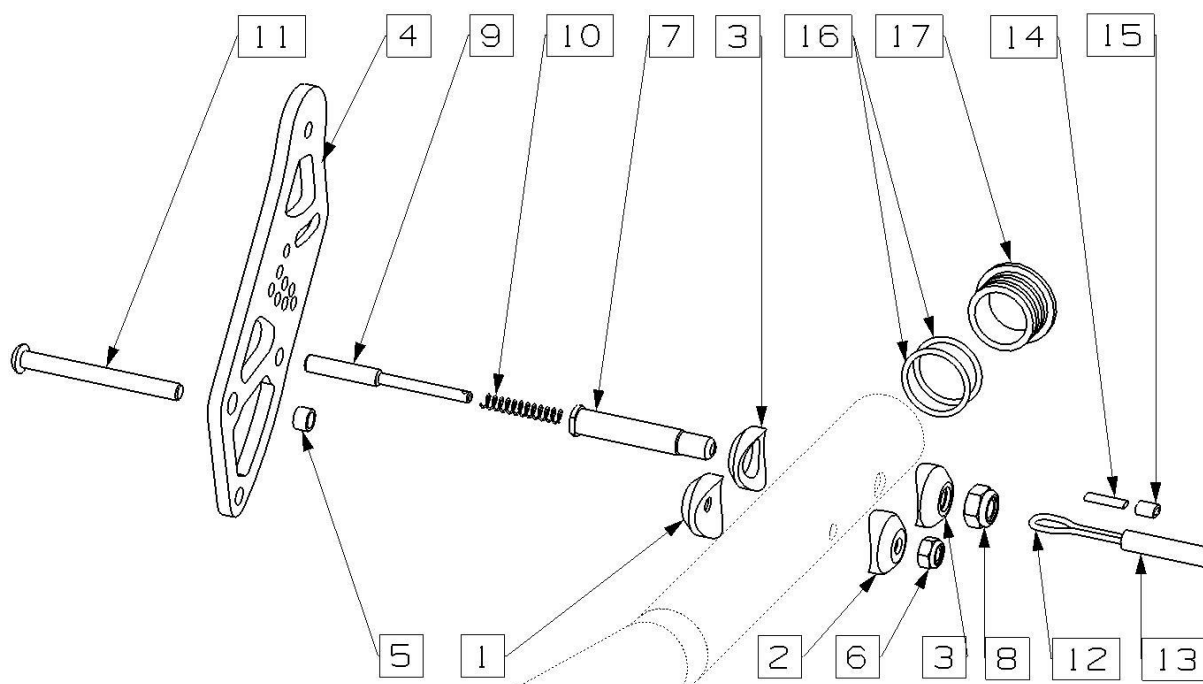

No.	Cat. No.	Element description	Qty
1	11-0310	Saddle washer	1
2	11-0309	Saddle washer	1
3	11-0307	Saddle washer	2
4	11-0546	Angle setting plate for right side	1
5	11-0405	Steel sleeve	1
6	NAKR_SAMOHAM_M6_CX	Self-locking nut M6	1
7	31-0576	Locking mechanism steel sleeve	1
8	NAKR_SAMOHAM_M8_CX	Self-locking nut M8	1
9	31-0580	Locking steel bolt	1
10	SPREZ_ZAMKA	Spring	1
11	SRU_ISO7380_M6X55	Bolt M6 x 55	1
12	-	Steel link	1
13	-	Plastic transparent link integument	1
14	73-0071	Plastic link cover	2
15	ZACISK_LINKI	Metal clips	2
16	ORING_21X2	O-ring	2
17	KAP_RAM_FI30_GRAW_L	Rear frame aluminum cap, right	1




Right side backrest mounting point of folding backrest mechanism for removable sideguards

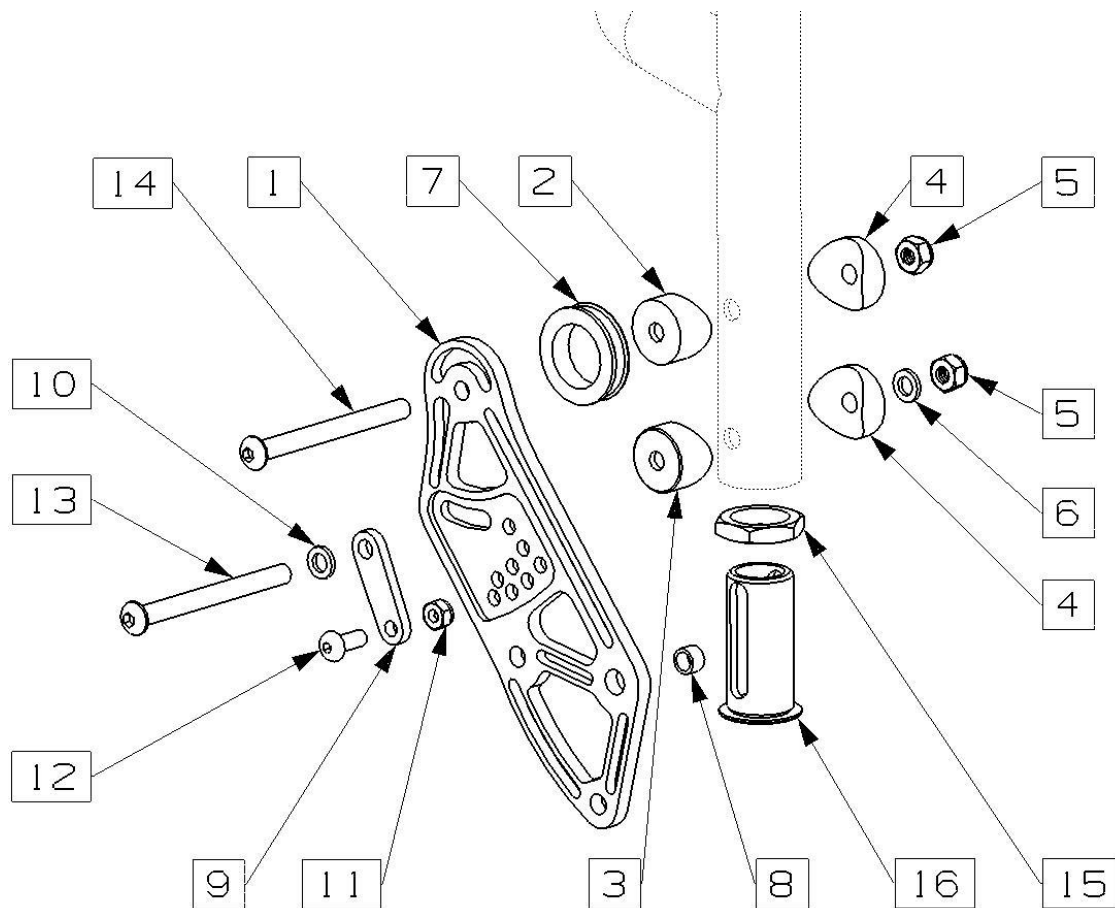

No.	Cat. No.	Element description	Qty
1	11-0546	Angle setting plate for right side	1
2	11-0304	Saddle washer for removable sideguards	1
3	11-0303	Saddle washer	1
4	11-0300	Saddle washer	2
5	NARK_SAMOHAM_M6_A2	Self-locking nut M6	2
6	PODKL_DIN125_A2_M6	Steel spacer	1
7	11-0774	Plastic receiver	1
8	11-0405	Steel sleeve	1
9	11-0158	Binder for angle setting plate	1
10	PODKL_DIN125_A2_M6	Steel spacer	1
11	31-0267	Self-locking nut M5	1
12	SRU_ISO7380_M5X12	Bolt M5 x 12	1
13	SRU_ISO7380_M6X60	Bolt M6 x 60	1
14	SRU_ISO7380_M6X55	BOLT M6 x 55	1
15	11-0576	Lock nut for backrest bumper	1
16	11-0397	Backrest bumper	1




Left side backrest mounting point of folding backrest mechanism for removable sideguards

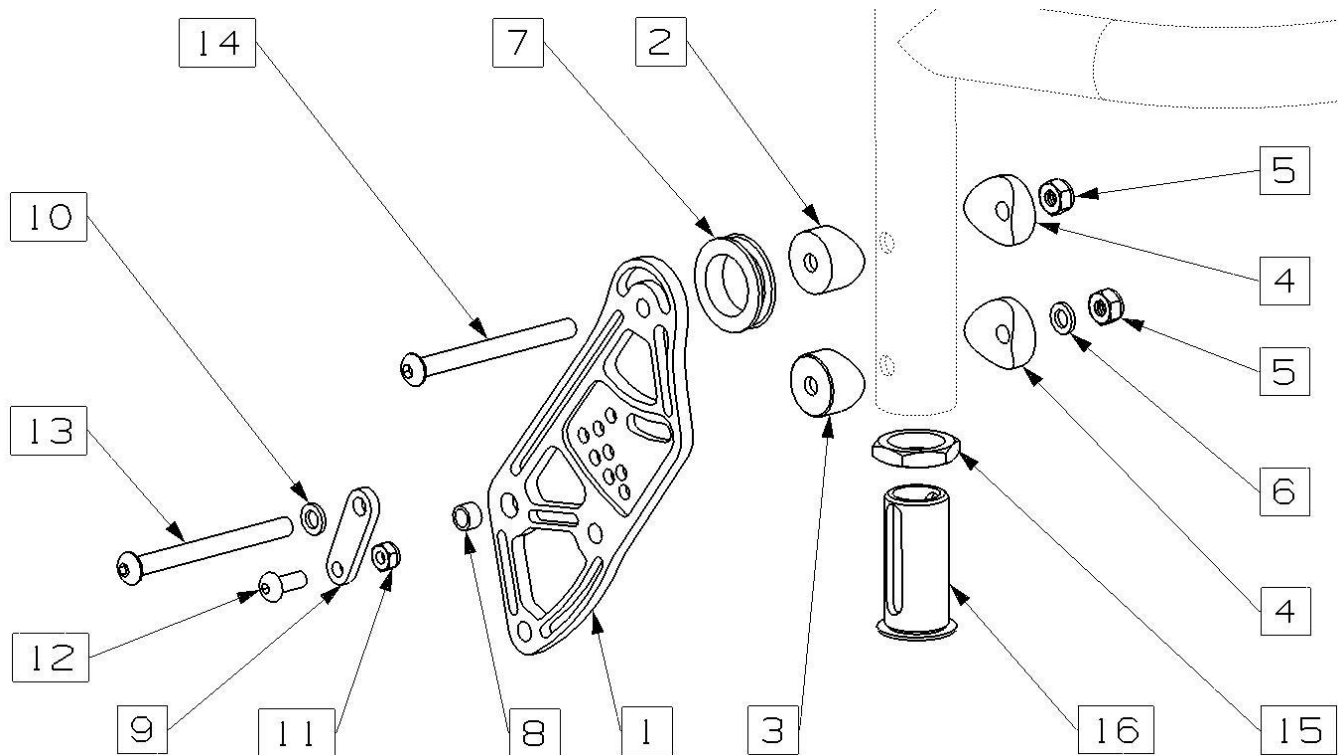

No.	Cat. No.	Element description	Qty
1	11-0545	Angle setting plate for left side	1
2	11-0304	Saddle washer for removable sideguards	1
3	11-0303	Saddle washer	1
4	11-0300	Saddle washer	2
5	NARK_SAMOHAM_M6_A2	Self-locking nut M6	2
6	PODKL_DIN125_A2_M6	Steel spacer	1
7	11-0774	Plastic receiver	1
8	11-0405	Steel sleeve	1
9	11-0158	Binder for angle setting plate	1
10	PODKL_DIN125_A2_M6	Steel spacer	1
11	31-0267	Self-locking nut M5	1
12	SRU_ISO7380_M5X12	Bolt M5 x 12	1
13	SRU_ISO7380_M6X60	Bolt M6 x 60	1
14	SRU_ISO7380_M6X55	Bolt M6 x 55	1
15	11-0576	Lock nut for backrest bumper	1
16	11-0397	Backrest bumper	1




Backrest handle mounting system

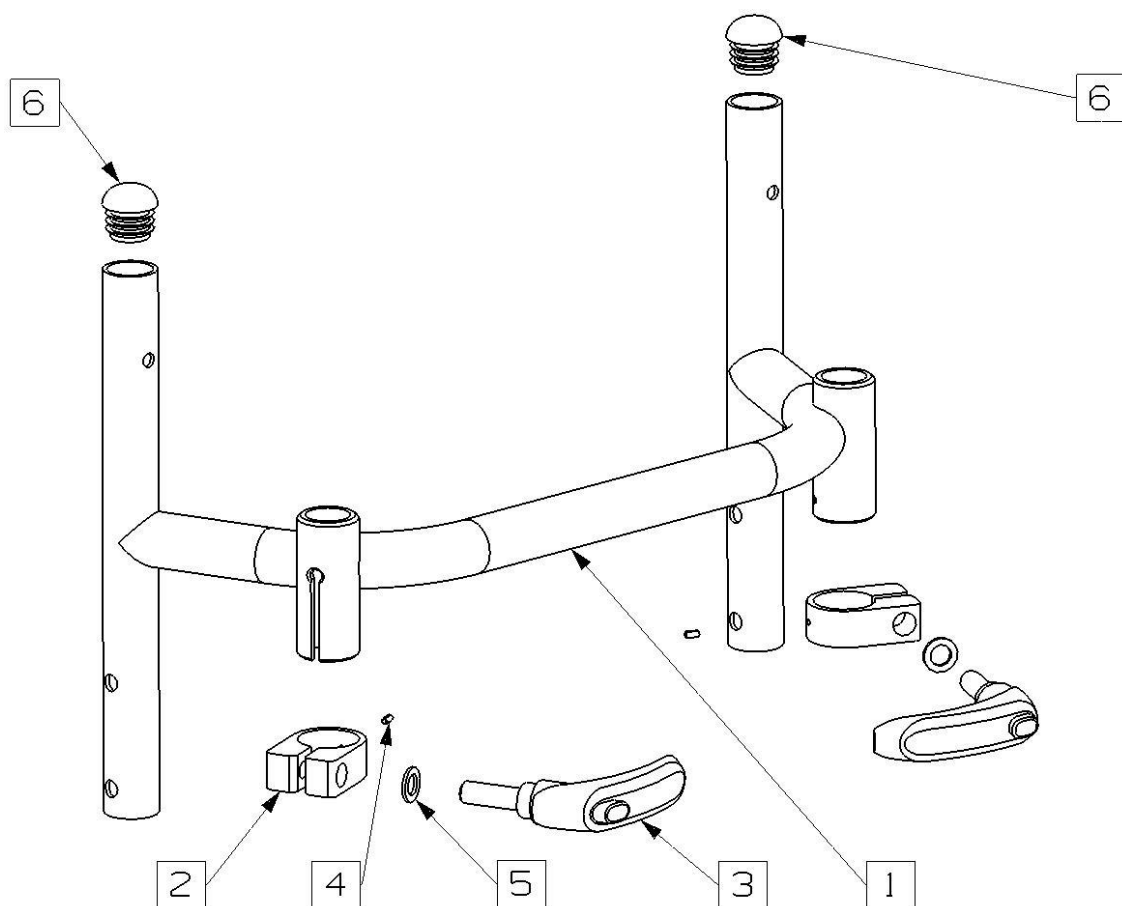

No.	Cat. No.	Element description	Qty
1	Individual part	Backrest tubes	1
2	11-0746	Push handles clamp, black anodised	2
3	SRU_RACZ_M10X35	Mounting handle	2
4	KOLEK_SPREZYSTY_3X6	Clamp pin	2
5	PODKL_DIN988_10_16_1	Handle spacer	2
6	KOR_KOPUL_R25	Plastic spherical caps	2



Aluminum pushhandle

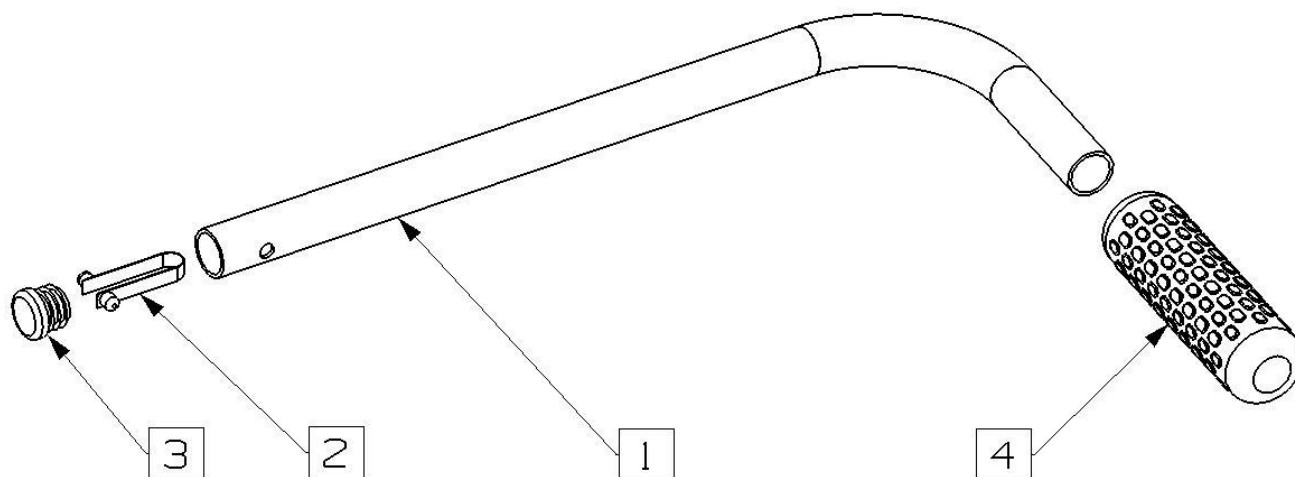


No.	Cat. No.	Element description	Qty
1	See part options below	Aluminum pushhandle tube	1
2	SPR_PLASKA	Flat spring	1
3	KOR_PLAS_R20	Black plastic 20mm flat cap	1
4	GRIP_STD	Standard grip	1

Part options – pushhandle tube

No.	Cat. No.	Element description	Qty
1 / A	11-0328	Aluminum pushhandle tube 33,5 cm long, black matt powder coated	1
1 / B	11-0330	Aluminum pushhandle tube 42 cm long, black matt powder coated	1
1 / C	11-0332	Aluminum pushhandle tube 43,5 cm long, black matt powder coated	1
1 / D	11-0334	Aluminum pushhandle tube 52,5 cm long, black matt powder coated	1




Backrest without pushhandles

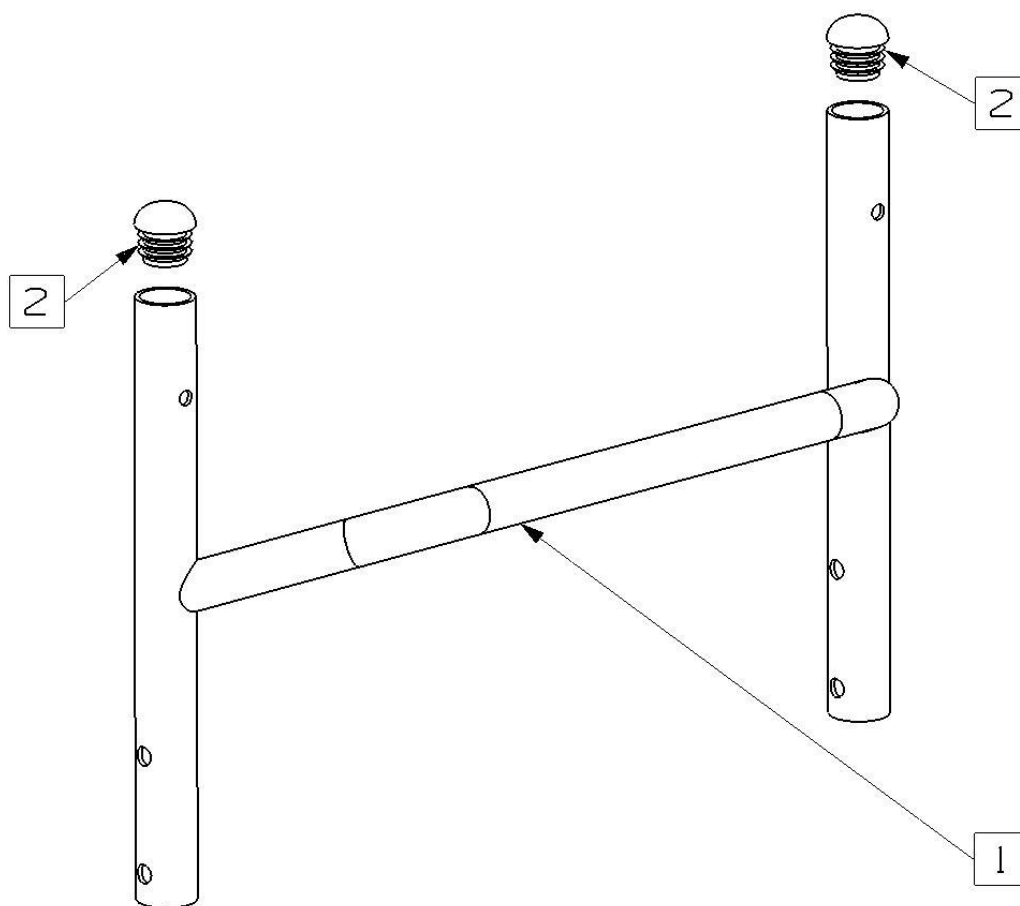

No.	Cat. No.	Element description	Qty
1	Individual part	Backrest tubes	1
2	KOR_KOPUL_R25	Plastic spherical caps	2




Backrest height adjusting system

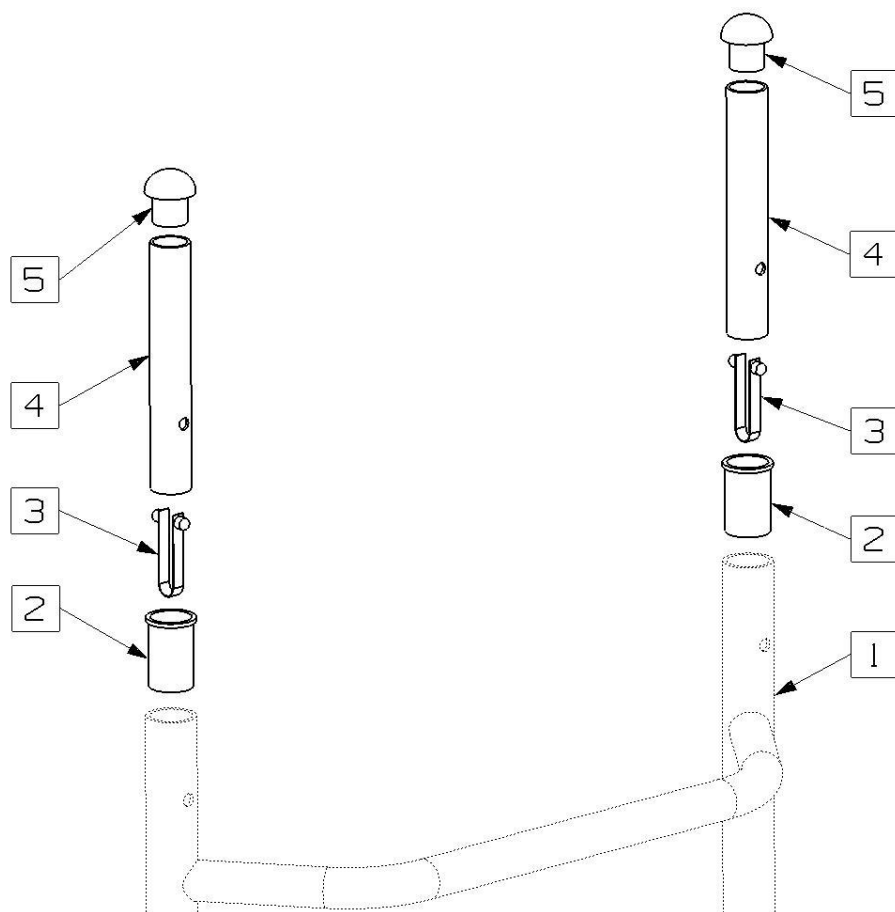

No.	Cat. No.	Element description	Qty
1	Individual part	Backrest tubes	1
2	WKL_REG_OPARCIA	Plastic sleeve	2
3	SPR_PLASKA	Flat spring	2
4	RUR_OPAR_20X125	Backrest height adjusting tubes	2
5	KOR_OPAR_KOPUL_25	Plastic spherical cap	2




Backrest with plastic foldable pushhandles

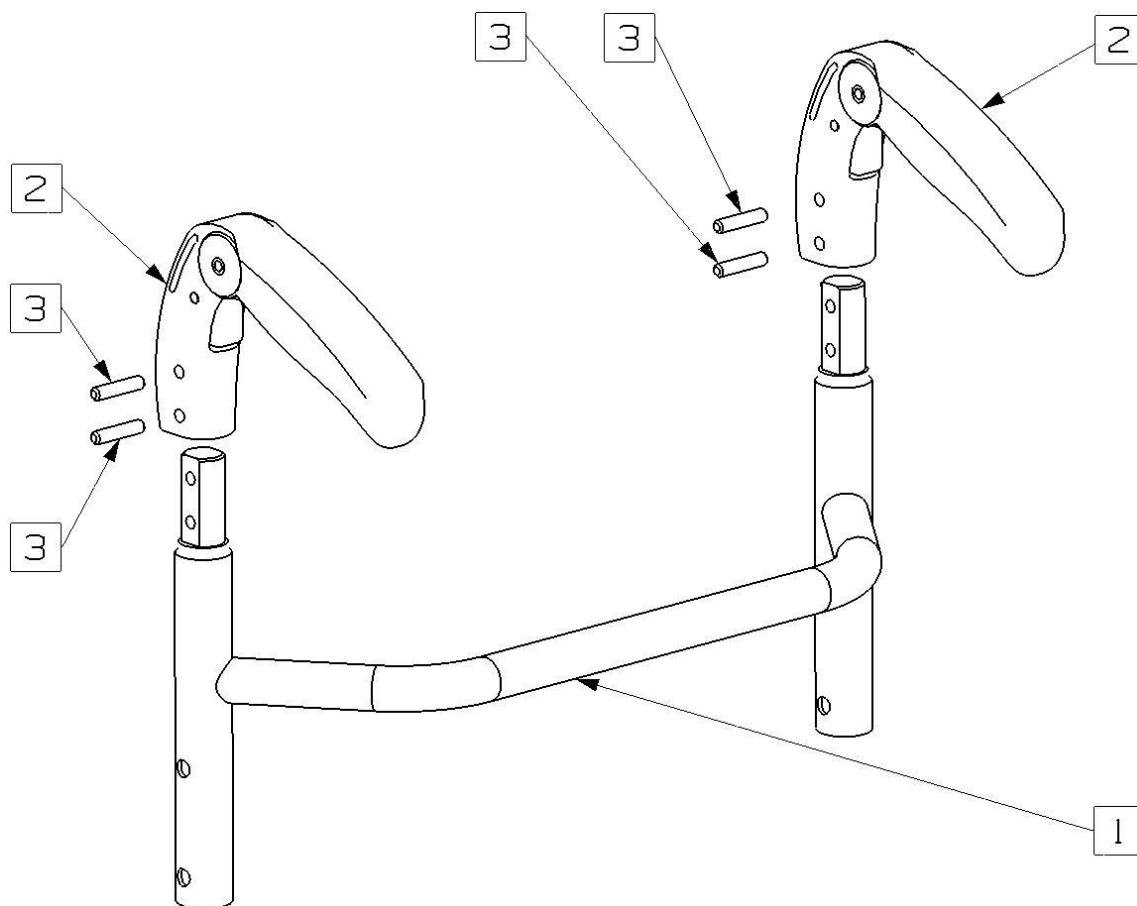

No.	Cat. No.	Element description	Qty
1	Individual part	Backrest tubes	1
2	RACZ_SKLAD_PLAST	Plastic foldable pushhandle	2
3	KOLEK_WALCOWY_ALU	Aluminum fixing pins	4




Backrest with screw-in pushhandles

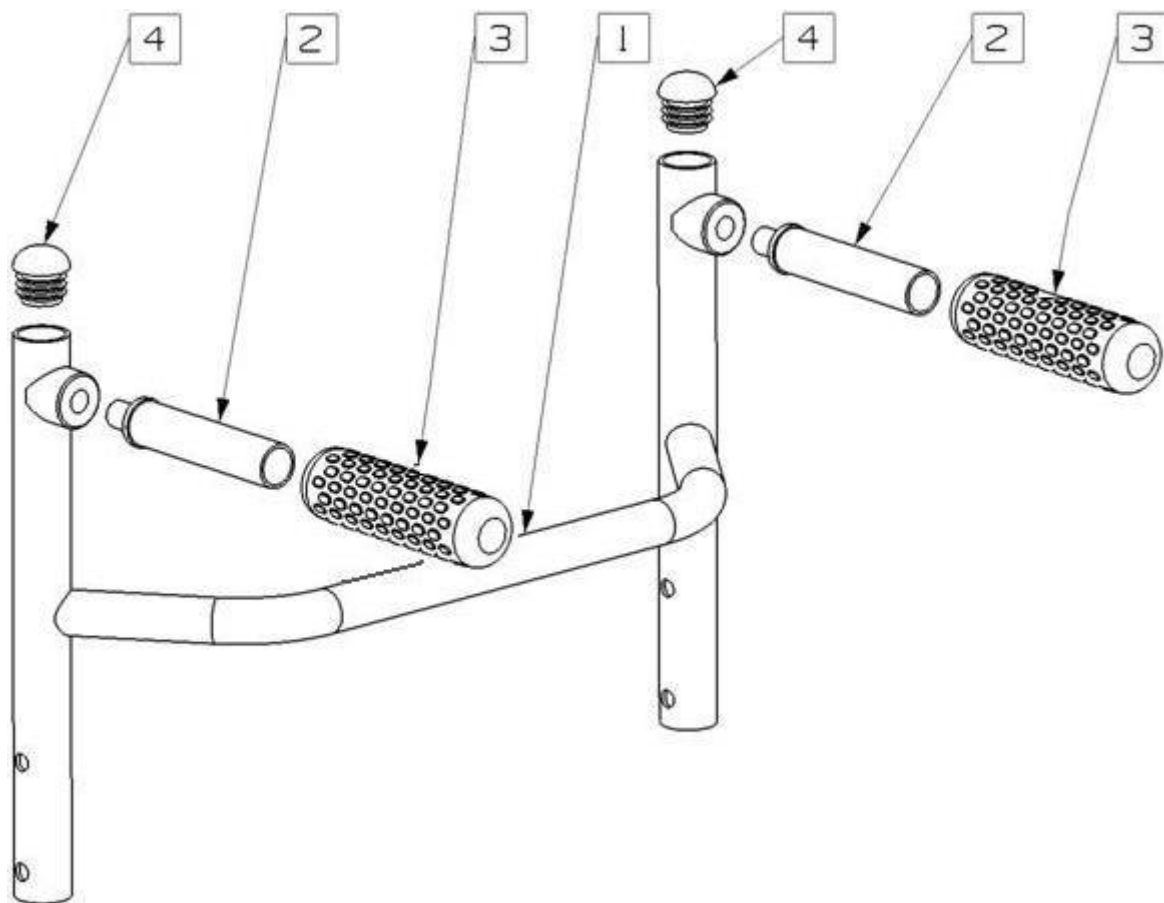

No.	Cat. No.	Element description	Qty
1	Individual part	Backrest tubes	1
2	11-0338	Steel handle	2
3	GRIP_STD	Rubber grip	2
4	KOR_KOPUL_R25	Plastic spherical caps	2




Backrest with integral pushhandles

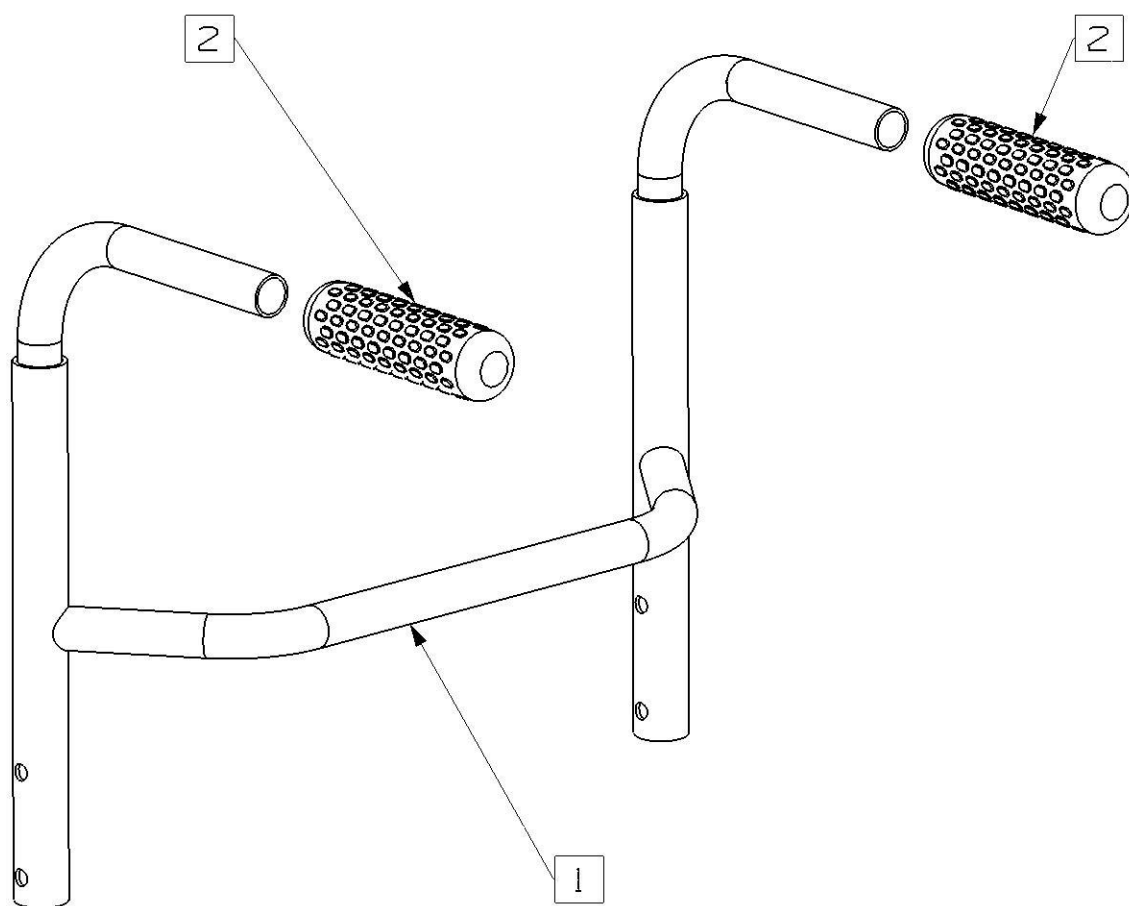

No.	Cat. No.	Element description	Qty
1	Individual part	Backrest tubes	1
2	GRIP_STD	Standard grip	2




Backrest with pushhandles in backrest tubes

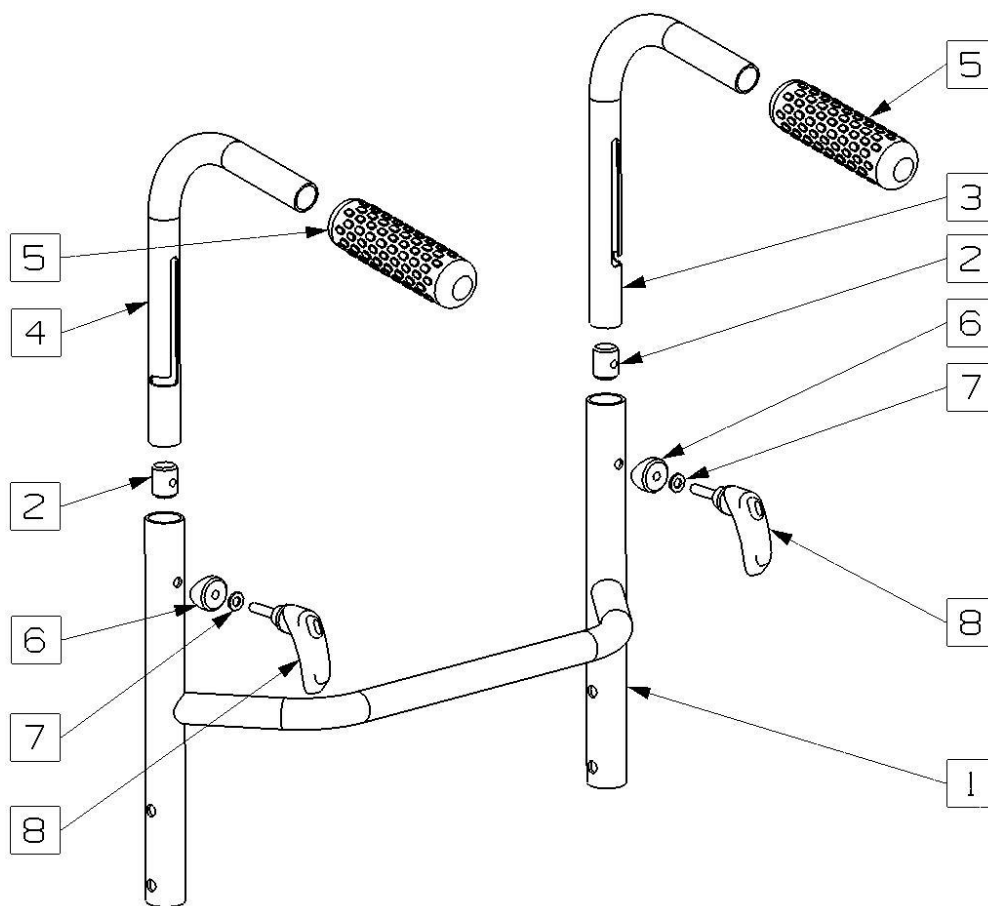

No.	Cat. No.	Element description	Qty
1	Individual part	Backrest tubes	1
2	BECZULKA_M6	Steel receiver	2
3	See part options below	Steel handle, right	1
4	See part options below	Steel handle, left	1
5	GRIP_STD	Standard grip	2
6	PUC_25_G6_F22/6	Saddle washer	2
7	PODKL_DIN125_A2_M6	Steel spacer	2
8	SRU_RACZ_MRX40_M6X25	Clamp M6	2



Camberbar clamps without antitipp receiver

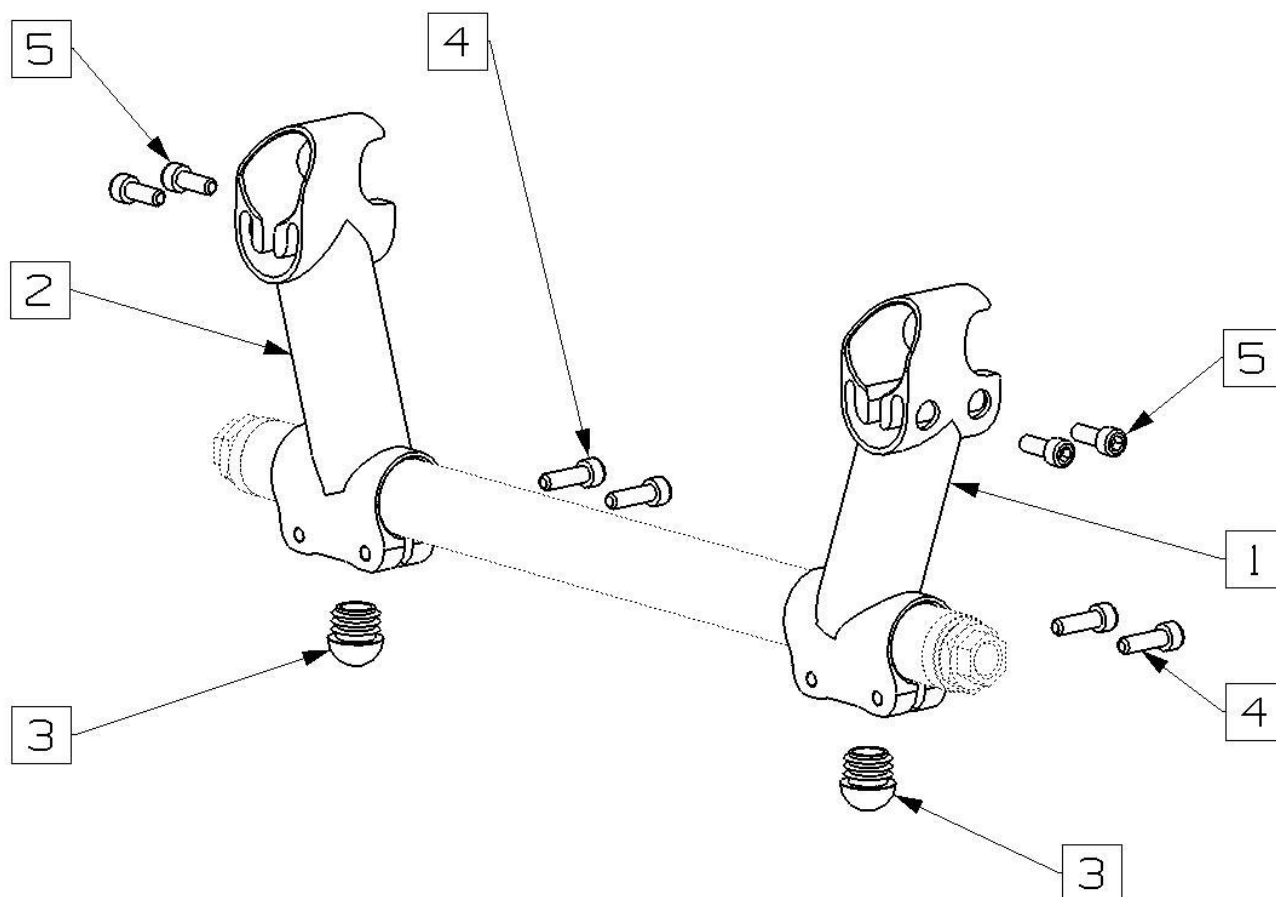

No.	Cat. No.	Element description	Qty
1	See part options below	Left camberbar clamp without antitipp receiver	1
2	See part options below	Right camberbar clamp without antitipp receiver	1
3	KOR_KOPUL_R20	Black plastic spherical cap	2X1
4	SRU_DIN912_M6X16	Screw DIN912 M6x16	2X2
5	SRU_DIN912_M6X20	Screw DIN912 M6x20	2X2

Part options – left camberbar clamp without antitipp receiver


No.	Cat. No.	Element description	Qty
1 / A	MOSTEK_MG07_STD_L_ST	Left camberbar clamp for rear seat height 44 cm, without antitipp receiver	1
1 / B	MOSTEK_MG10_STD_L_ST	Left camberbar clamp for rear seat height 42 cm, without antitipp receiver	1
1 / C	MOSTEK_MG13_STD_L_ST	Left camberbar clamp for rear seat height 40 cm, without antitipp receiver	1

Part options – right camberbar clamp without antitipp receiver

No.	Cat. No.	Element description	Qty
2 / A	MOSTEK_MG07_STD_P_ST	Right camberbar clamp for rear seat height 44 cm, without antitipp receiver	1
2 / B	MOSTEK_MG10_STD_P_ST	Right camberbar clamp for rear seat height 42 cm, without antitipp receiver	1
2 / C	MOSTEK_MG13_STD_P_ST	Right camberbar clamp for rear seat height 40 cm, without antitipp receiver	1



Camberbar clamps with antitipp receiver

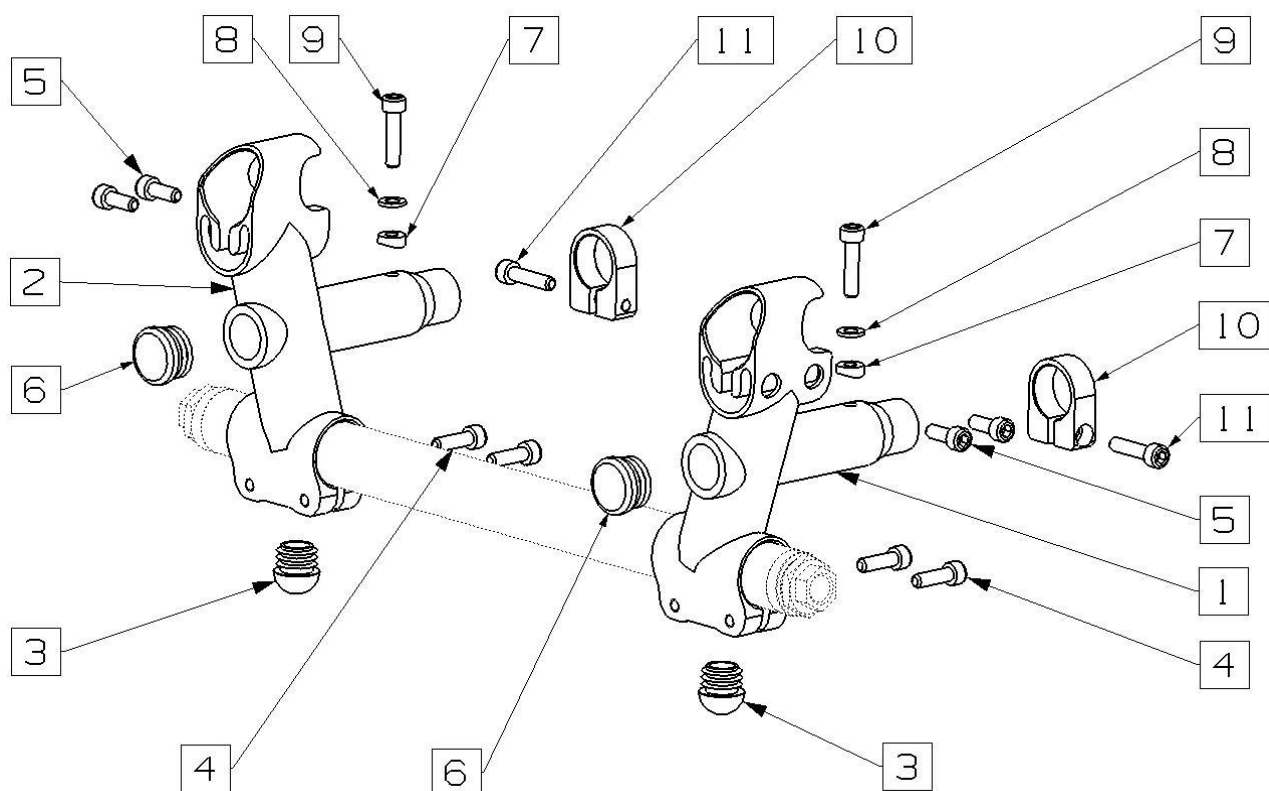

No.	Cat. No.	Element description	Qty
1	See part options below	Left camberbar clamp with antitipp receiver	1
2	See part options below	Right camberbar clamp with antitipp receiver	1
3	KOR_KOPUL_R20	Black plastic spherical cap	2 X 1
4	SRU_DIN912_M6X16	Screw DIN912 M6x16	2 X 2
5	SRU_DIN912_M6X20	Screw DIN912 M6x20	2 X 2
6	KOR_PLAS_R25	Black plastic 25mm Flat cap	2 X 1
7	PUC_R25_G3_F12/6	Saddle washer	2 X 1
8	PODKL_DIN125_A2_M6	Steel spacer	2 X 1
9	SRU_DIN912_M6X25	Screw DIN912 M6x25	2 X 1
10	11-0522	Antitipper clamp	2 X 1
11	SRU_DIN912_M6X25	Screw DIN912 M6x25	2 X 1

Part options – left camberbar clamp with antitipp receiver


No.	Cat. No.	Element description	Qty
1 / A	MOSTEK_MG07_STD_L_AT	Left camberbar clamp for rear seat height 44 cm, with antitipp receiver	1
1 / B	MOSTEK_MG10_STD_L_AT	Left camberbar clamp for rear seat height 42 cm, with antitipp receiver	1
1 / C	MOSTEK_MG13_STD_L_AT	Left camberbar clamp for rear seat height 40 cm, with antitipp receiver	1

Part options – right camberbar clamp with antitipp receiver

No.	Cat. No.	Element description	Qty
2 / A	MOSTEK_MG07_STD_P_AT	Right camberbar clamp for rear seat height 44 cm, with antitipp receiver	1
2 / B	MOSTEK_MG10_STD_P_AT	Right camberbar clamp for rear seat height 42 cm, with antitipp receiver	1
2 / C	MOSTEK_MG13_STD_P_AT	Right camberbar clamp for rear seat height 40 cm, with antitipp receiver	1



Camberbar clamps for rear COG without antitipp receiver

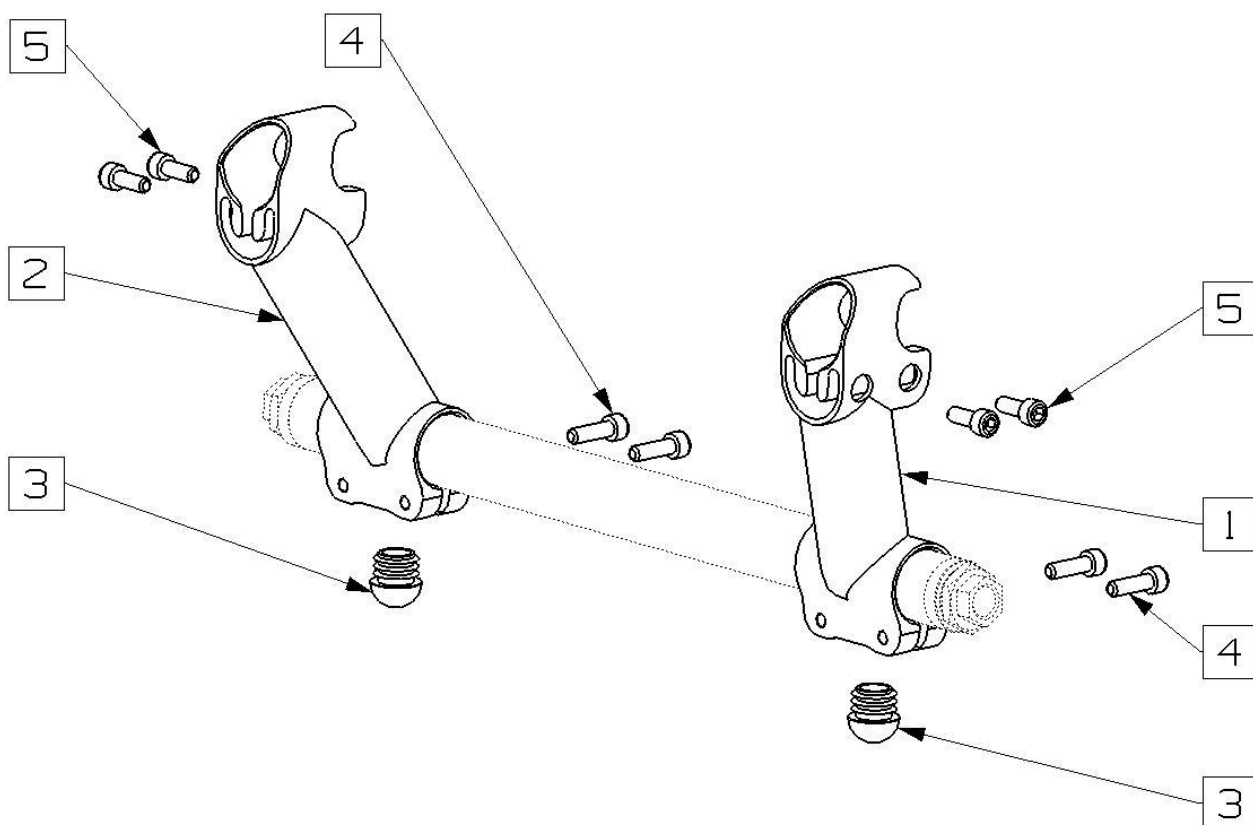

No.	Cat. No.	Element description	Qty
1	See part options below	Left camberbar clamp for rear COG without antitipp receiver	1
2	See part options below	Right camberbar clamp for rear COG without antitipp receiver	1
3	KOR_KOPUL_R20	Black plastic spherical cap	2X1
4	SRU_DIN912_M6X16	Screw DIN912 M6x16	2X2
5	SRU_DIN912_M6X20	Screw DIN912 M6x20	2X2

Part options – left camberbar clamp for rear COG without antitipp receiver

No.	Cat. No.	Element description	Qty
1 / A	MOSTEK_MG07_ZER_L_ST	Left camberbar clamp for rear COG for rear seat height 44 cm, without antitipp receiver	1
1 / B	MOSTEK_MG10_ZER_L_ST	Left camberbar clamp for rear COG for rear seat height 42 cm, without antitipp receiver	1
1 / C	MOSTEK_MG13_ZER_L_ST	Left camberbar clamp for rear COG for rear seat height 40 cm, without antitipp receiver	1


Part options – right camberbar clamp for rear COG without antitipp receiver

No.	Cat. No.	Element description	Qty
2 / A	MOSTEK_MG07_ZER_P_ST	Right camberbar clamp for rear COG for rear seat height 44 cm, without antitipp receiver	1
2 / B	MOSTEK_MG10_ZER_P_ST	Right camberbar clamp for rear COG for rear seat height 42 cm, without antitipp receiver	1
2 / C	MOSTEK_MG13_ZER_P_ST	Right camberbar clamp for rear COG for rear seat height 40 cm, without antitipp receiver	1





Camberbar clamps for rear COG with antitipp receiver

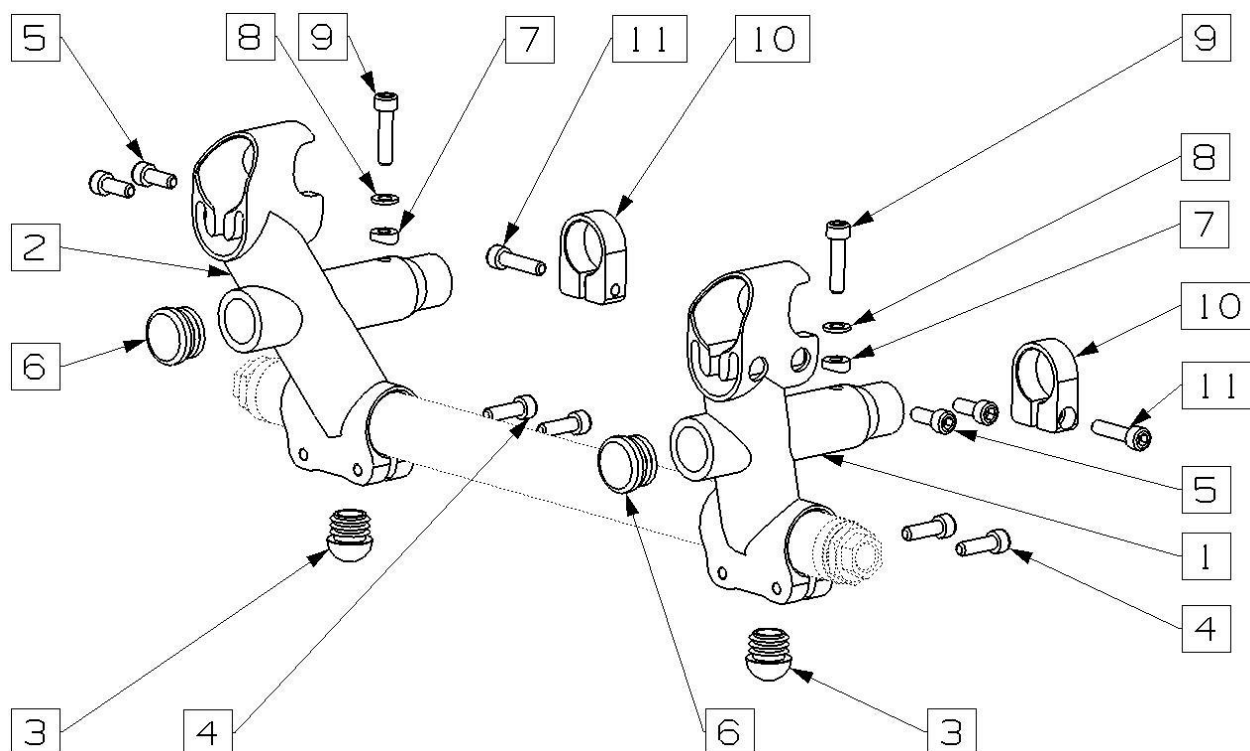

No.	Cat. No.	Element description	Qty
1	See part options below	Left camberbar clamp for rear COG with antitipp receiver	1
2	See part options below	Right camberbar clamp for rear COG with antitipp receiver	1
3	KOR_KOPUL_R20	Black plastic spherical cap	2 X 1
4	SRU_DIN912_M6X16	Screw DIN912 M6x16	2 X 2
5	SRU_DIN912_M6X20	Screw DIN912 M6x20	2 X 2
6	KOR_PLAS_R25	Black plastic 25mm Flat cap	2 X 1
7	PUC_R25_G3_F12/6	Saddle washer	2 X 1
8	PODKL_DIN125_A2_M6	Steel spacer	2 X 1
9	SRU_DIN912_M6X25	Screw DIN912 M6x25	2 X 1
10	11-0522	Antitipper clamp	2 X 1
11	SRU_DIN912_M6X25	Screw DIN912 M6x25	2 X 1

Part options – left camberbar clamp with antitipp receiver


No.	Cat. No.	Element description	Qty
1 / A	MOSTEK_MG07_ZER_L_AT	Left camberbar clamp for rear COG for rear seat height 44 cm, with antitipp receiver	1
1 / B	MOSTEK_MG10_ZER_L_AT	Left camberbar clamp for rear COG for rear seat height 42 cm, with antitipp receiver	1
1 / C	MOSTEK_MG13_ZER_L_AT	Left camberbar clamp for rear COG for rear seat height 40 cm, with antitipp receiver	1

Part options – right camberbar clamp with antitipp receiver

No.	Cat. No.	Element description	Qty
2 / A	MOSTEK_MG07_ZER_P_AT	Right camberbar clamp for rear COG for rear seat height 44 cm, with antitipp receiver	1
2 / B	MOSTEK_MG10_ZER_P_AT	Right camberbar clamp for rear COG for rear seat height 42 cm, with antitipp receiver	1
2 / C	MOSTEK_MG13_ZER_P_AT	Right camberbar clamp for rear COG for rear seat height 40 cm, with antitipp receiver	1



Camberbar

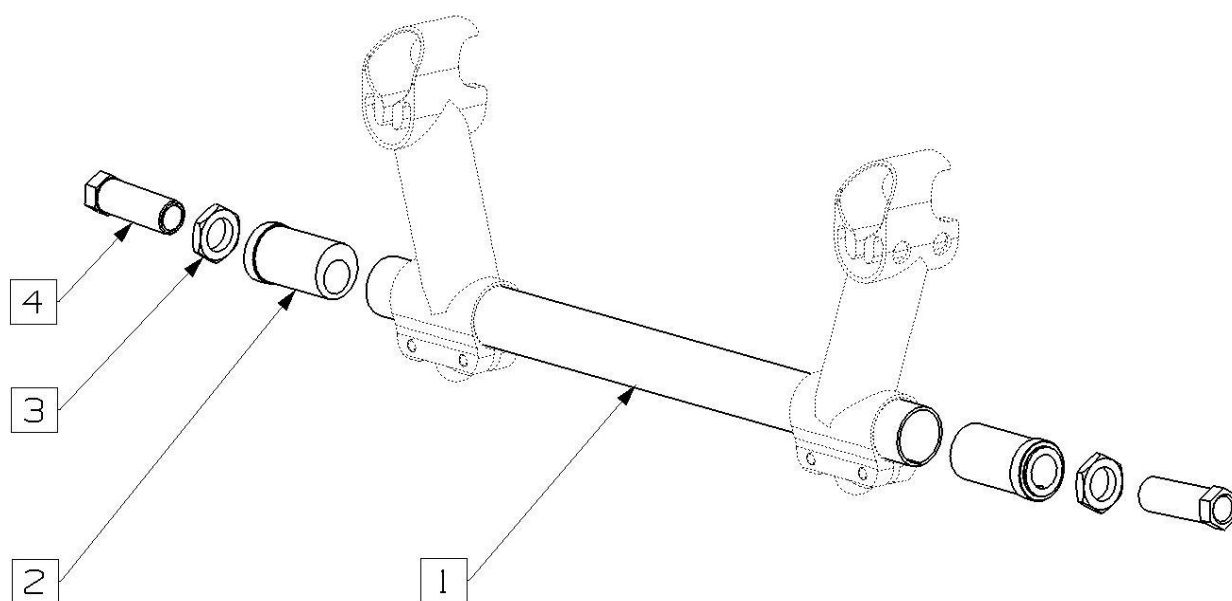

No.	Cat. No.	Element description	Qty
1	See part options below	Camberbar tube	1
2	See part options below	Camber inserts	2
3	NAKR_ALU_16X1_WOL	Nut M16x1	2
4	See part options below	Steel sleeve	2

Part options – camberbar tube

No.	Cat. No.	Element description	Qty
1 / A	BELKA_30	Camberbar tube for seat width 30 cm, black matt powder coated	1
1 / B	BELKA_31	Camberbar tube for seat width 31 cm, black matt powder coated	1
1 / C	BELKA_32	Camberbar tube for seat width 32 cm, black matt powder coated	1
1 / D	BELKA_33	Camberbar tube for seat width 33 cm, black matt powder coated	1
1 / E	BELKA_34	Camberbar tube for seat width 34 cm, black matt powder coated	1
1 / F	BELKA_35	Camberbar tube for seat width 35 cm, black matt powder coated	1
1 / G	BELKA_36	Camberbar tube for seat width 36 cm, black matt powder coated	1
1 / H	BELKA_37	Camberbar tube for seat width 37 cm, black matt powder coated	1
1 / I	BELKA_38	Camberbar tube for seat width 38 cm, black matt powder coated	1
1 / J	BELKA_39	Camberbar tube for seat width 39 cm, black matt powder coated	1
1 / K	BELKA_40	Camberbar tube for seat width 40 cm, black matt powder coated	1
1 / L	BELKA_41	Camberbar tube for seat width 41 cm, black matt powder coated	1
1 / M	BELKA_42	Camberbar tube for seat width 42 cm, black matt powder coated	1
1 / N	BELKA_43	Camberbar tube for seat width 43 cm, black matt powder coated	1
1 / O	BELKA_44	Camberbar tube for seat width 44 cm, black matt powder coated	1
1 / P	BELKA_45	Camberbar tube for seat width 45 cm, black matt powder coated	1
1 / Q	BELKA_46	Camberbar tube for seat width 46 cm, black matt powder coated	1
1 / R	BELKA_47	Camberbar tube for seat width 47 cm, black matt powder coated	1
1 / S	BELKA_48	Camberbar tube for seat width 48 cm, black matt powder coated	1
1 / T	BELKA_49	Camberbar tube for seat width 49 cm, black matt powder coated	1
1 / U	BELKA_50	Camberbar tube for seat width 50 cm, black matt powder coated	1

Part options – angle inserts


No.	Cat. No.	Element description	Qty
2 / A	KAMB_OST_25_H	Angle insert 0°	2
2 / B	KAMB_1ST_25_H	Angle insert 1°	2
2 / C	KAMB_2ST_25_H	Angle insert 2°	2
2 / D	KAMB_3ST_25_H	Angle insert 3°	2
2 / E	KAMB_4ST_25_H	Angle insert 4°	2
2 / F	KAMB_5ST_25_H	Angle insert 5°	2

Part options – steel sleeve

No.	Cat. No.	Element description	Qty
4 / A	TULEJ_M16X1X50	Sleeve for axle 12 mm	2
4 / B	TULEJ_M16X1X50_CAL	Sleeve for axle 12,7 mm (½")	2



Standard wheel 24" with radial spokes

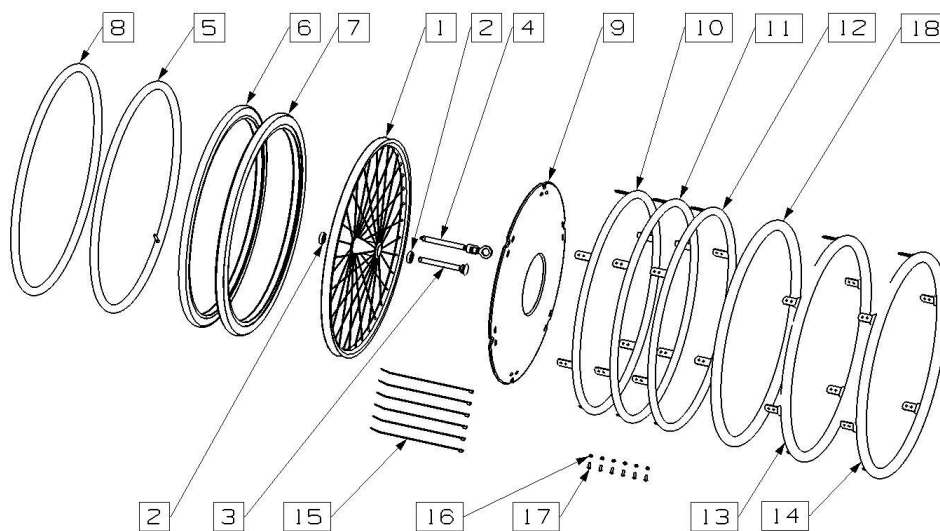

No.	Cat. No.	Element description	Qty
1	-	Standard radial wheel 24"	2 X 1
2	See options parts list	Ball-bearing	2 X 2
3	See options parts list	Axle with cap	2 X 1
4	See options parts list	Axle with tetra handle	2 X 1
5	DETKA_24_S_1	Tube 24" for tyres	2 X 1
6	See options parts list	Schwalbe RightRun 24" tyre	2 X 1
7	OPONA_24_SM+_CZ	Schwalbe Marathon Plus 24" tyre	2 X 1
8	OPONA_24_TU_WS3_RR	Tufo 24" tubeless tyre	2 X 1
9	See options parts list	Spokesguard 24"	2 X 1
10	CALU_24	Aluminum pushrim 24"	2 X 1
11	CSTAL_24	Steel pushrim 24"	2 X 1
12	CTYT_24_352485	Titanium pushrim 24"	2 X 1
13	CSTAL_24_POWL	Standard rubber covered pushrim 24"	2 X 1
14	CIAG_24_TETRAGRIP	Tetra silicone covered pushrim 24"	2 X 1
15	OP_ZAC_250/4,8_BIA	Spokesguard connector	2 X 6
16	SRU_ISO7380_M5X10	Screw M5x10 for pushrim mounting	2 X 6
17	NAKR_SAMOHAM_M5	Self-locking nut M6 for pushrim mounting	2 X 6
18	NAKLAD_NA_C_24	Rubber cover for pushrims: aluminum, steel, titanium	2 X 1

Part options – ball bearings

No.	Cat. No.	Element description	Qty
2 / A	LOZ_28-12-8-6001CX	Ball bearing for 12 mm axles	2 X 2
2 / B	LOZ_MC_28X12,8_CX	Ball bearing for 12,7 mm (1/2") axles	2 X 2

Part options – axles with and without caps

No.	Cat. No.	Element description	Qty
3 / A	OS_12X98_B_KAP	12 mm axle with cap for standard radial wheels	2 X 1
3 / B	OS_12,8X98_B_KAP	12,7 mm (1/2") axle without cap for standard radial wheels	2 X 1

Part options – axles with tetra handle

No.	Cat. No.	Element description	Qty
4 / A	OS_12X98_Z_KAP_Q	12 mm axle with tetra handle for standard radial wheels	2 X 1
4 / B	OS_12,8X98_Z_KAP_Q	12,7 mm (1/2") axle with tetra handle for standard radial wheels	2 X 1

Part options – Schwalbe RightRun tyres

No.	Cat. No.	Element description	Qty
6 / A	OPONA_24_SRR_SZ	Schwalbe RightRun 24" tyre, gray	2 X 1
6 / C	OPONA_24_SRR_CE	Schwalbe RightRun 24" tyre, red	2 X 1
6 / D	OPONA_24_SRR_N	Schwalbe RightRun 24" tyre, blue	2 X 1


Part options – spokesguards

No.	Cat. No.	Element description	Qty
9 / A	OSL_SPR_24_BBARW	Spokesguard 24", transparent	2 X 1
9 / B	OSL_SPR_24_GTM	Spokesguard 24", transparent with GTM Logo	2 X 1

Standard wheel 24" with crossed spokes



GTM-MOBIL M-T T. Walisiewicz Sp. Komandytowa • ul. Dorodna 16, 03-195 Warszawa • www.gtmobil.com.pl
 service@gtmmobil.com.pl • foreign.office@gtmmobil.com.pl • tel./fax +48 022 504 12 75

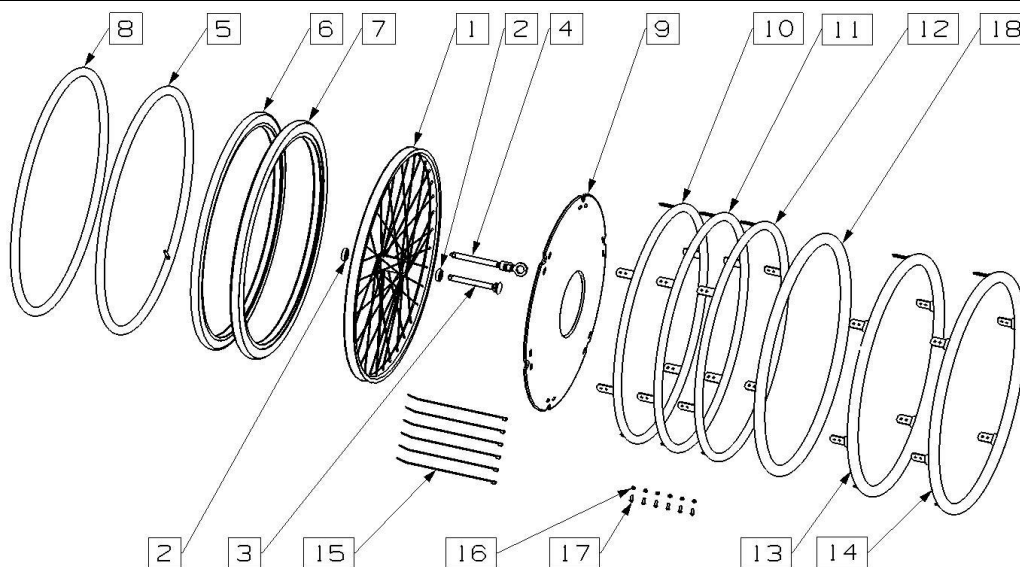

No.	Cat. No.	Element description	Qty
1	-	Standard crossed wheel 24"	2 X 1
2	See options parts list	Ball-bearing	2 X 2
3	See options parts list	Axle with cap	2 X 1
4	See options parts list	Axle with tetra handle	2 X 1
5	DETKA_24_S_1	Tube 24" for tyres	2 X 1
6	See options parts list	Schwalbe RightRun 24" tyre	2 X 1
7	OPONA_24_SM+_CZ	Schwalbe Marathon Plus 24" tyre	2 X 1
8	OPONA_24_TU_WS3_RR	Tufo 24" tubeless tyre	2 X 1
9	See options parts list	Spokesguard 24"	2 X 1
10	CALU_24	Aluminum pushrim 24"	2 X 1
11	CSTAL_24	Steel pushrim 24"	2 X 1
12	CTYT_24_352485	Titanium pushrim 24"	2 X 1
13	CSTAL_24_POWL	Standard rubber covered pushrim 24"	2 X 1
14	CIAG_24_TETRAGRIP	Tetra silicone covered pushrim 24"	2 X 1
15	OP_ZAC_250/48_BIA	Spokesguard connector	2 X 6
16	SRU_ISO7380_M5X10	Screw M5x10 for pushrim mounting	2 X 6
17	NAKR_SAMOHAM_M5	Self-locking nut M6 for pushrim mounting	2 X 6
18	NAKLAD_NA_C_24	Rubber cover for pushrims: aluminum, steel, titanium	2 X 1

Part options – ball bearings

No.	Cat. No.	Element description	Qty
2 / A	LOZ_28_12-8_6001CX	Ball bearing for 12 mm axles	2 X 2
2 / B	LOZ_MC_28X12,8_CX	Ball bearing for 12,7 mm (1/2") axles	2 X 2

Part options – axles with and without caps

No.	Cat. No.	Element description	Qty
3 / A	OS_12X98_K_KROTOKI	12 mm axle with cap for standard radial wheels	2 X 1
3 / B	OS_12,8X98_B_KAP	12,7 mm (1/2") axle without cap for standard radial wheels	2 X 1

Part options – axles with tetra handle

No.	Cat. No.	Element description	Qty
4 / A	OS_12X98_Z_KAP_Q	12 mm axle with tetra handle for standard radial wheels	2 X 1
4 / B	OS_12,8X98_Z_KAP_Q	12,7 mm (1/2") axle with tetra handle for standard radial wheels	2 X 1

Part options – Schwalbe RightRun tyres

No.	Cat. No.	Element description	Qty
6 / A	OPONA_24_SRR_SZ	Schwalbe RightRun 24" tyre, gray	2 X 1
6 / C	OPONA_24_SRR_CE	Schwalbe RightRun 24" tyre, red	2 X 1
6 / D	OPONA_24_SRR_N	Schwalbe RightRun 24" tyre, blue	2 X 1


Part options – spokesguards

No.	Cat. No.	Element description	Qty
9 / A	OSL_SPR_24_BBARW	Spokesguard 24", transparent	2 X 1
9 / B	OSL_SPR_24_GTM	Spokesguard 24", transparent with GTM Logo	2 X 1

Sport wheel 24" with crossed spokes



GTM-MOBIL M-T T. Walisiewicz Sp. Komandytowa • ul. Dorodna 16, 03-195 Warszawa • www.gtmobil.com.pl
 service@gtmmobil.com.pl • foreign.office@gtmmobil.com.pl • tel./fax +48 022 504 12 75

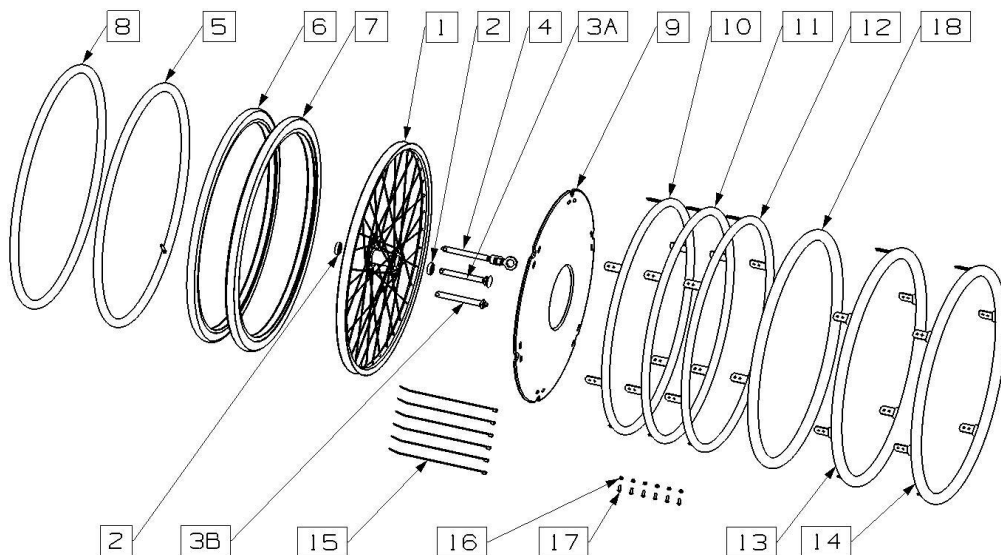

No.	Cat. No.	Element description	Qty
1	KOL_24_KRZ	Sport crossed wheel 24"	2 X 1
2	See options parts list	Ball-bearing	2 X 2
3A	OS_12X103_Z_KAP	Axle 12 mm with cap	2 X 1
3B	OS_12,8X104_B_KAP	Axle 12,7 mm (1/2") without cap	2 X 1
4	See options parts list	Axle with tetra handle	2 X 1
5	DETKA_24_S_1	Tube 24" for tyres	2 X 1
6	OPONA_24_SS	Schwalbe Speedair tyre	2 X 1
7	OPONA_24_SM+_CZ	Schwalbe Marathon Plus 24" tyre	2 X 1
8	OPONA_24_TU_WS3_RR	Tufo 24" tubeless tyre	2 X 1
9	See options parts list	Spokesguard 24"	2 X 1
10	CALU_24	Aluminum pushrim 24"	2 X 1
11	CSTAL_24	Steel pushrim 24"	2 X 1
12	CTYT_24_352485	Titanium pushrim 24"	2 X 1
13	CSTAL_24_POWL	Standard rubber covered pushrim 24"	2 X 1
14	CIAG_24_TETRAGRIP	Tetra silicone covered pushrim 24"	2 X 1
15	OP_ZAZ_250/48_BIA	Spokesguard connector	2 X 6
16	SRU_ISO7380_M5X10	Screw M5x10 for pushrim mounting	2 X 6
17	NAKR_SAMOHAM_M5	Self-locking nut M6 for pushrim mounting	2 X 6
18	NAKLAD_NA_C_24	Rubber cover for pushrims: aluminum, steel, titanium	2 X 1

Part options – ball bearings

No.	Cat. No.	Element description	Qty
2 / A	LOZ_28-12-8_6001	Ball bearing for 12 mm axles	2 X 2
2 / B	LOZ_MC_28X12,8_CX	Ball bearing for 12,7 mm (1/2") axles	2 X 2

Part options – axles with tetra handle


No.	Cat. No.	Element description	Qty
4 / A	OS_12X104_Z_KAP_Q	12 mm axle with tetra handle for sport crossed wheels	2 X 1
4 / B	-OS_12,8X104_Z_KAP_Q	12,7 mm (1/2") axle with tetra handle for sport crossed wheels	2 X 1

Part options – spokesguards

No.	Cat. No.	Element description	Qty
9 / A	OSL_SPR_24_BBARW	Spokesguard 24", transparent	2 X 1
9 / B	OSL_SPR_24_GTM	Spokesguard 24", transparent with GTM Logo	2 X 1



All-terrain wheel 24" with radial spokes

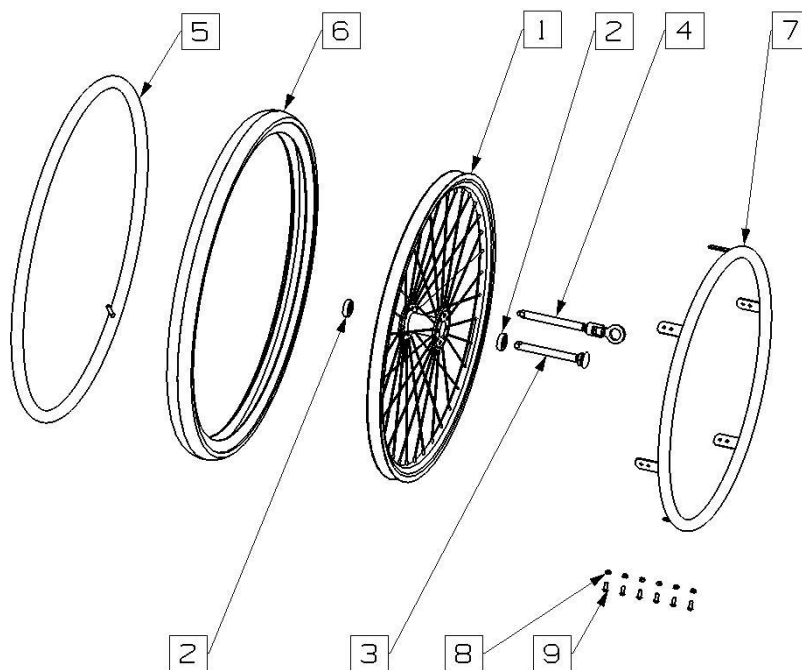

No.	Cat. No.	Element description	Qty
1	KOL_24_TER_CALU_CA	All-terrain radial wheel 24"	2 X 1
2	See options parts list	Ball-bearing	2 X 2
3	See options parts list	Axle with cap	2 X 1
4	See options parts list	Axle with tetra handle	2 X 1
5	DETKA_22_TERENOWA	Tube 24" for All-terrain tyres	2 X 1
6	OPONA_24_S_TEREN	All-terrain 24" tyre	2 X 1
7	-	Aluminum pushrim 24" for All-terrain wheels, black anodised	2 X 1
8	SRU_ISO7380_M5X10	Screw M5x10 for pushrim mounting	2 X 6
9	NAKR_SAMOHAM_M5	Self-locking nut M6 for pushrim mounting	2 X 6

Part options – ball bearings

No.	Cat. No.	Element description	Qty
2 / A	LOZ_28-12-8_6001	Ball bearing for 12 mm axles	2 X 2
2 / B	LOZ_MC_28X12,8_CX	Ball bearing for 12,7 mm (1/2") axles	2 X 2

Part options – axles with or without caps


No.	Cat. No.	Element description	Qty
3 / A	OS_12X103_Z_KAP	12 mm axle with cap for all-terrain radial wheels	2 X 1
3 / B	OS_12,8X104_B_KAP	12,7 mm (1/2") axle without cap for all-terrain radial wheels	2 X 1

Part options – axles with tetra handle

No.	Cat. No.	Element description	Qty
4 / A	OS_12X104_Z_KAP_Q	12 mm axle with tetra handle for all-terrain radial wheels	2 X 1
4 / B	OS_12,8X104_Z_KAP_Q	12,7 mm (1/2") axle with tetra handle for all-terrain radial wheels	2 X 1



Spinergy L wheel 24" with 18 radial spokes

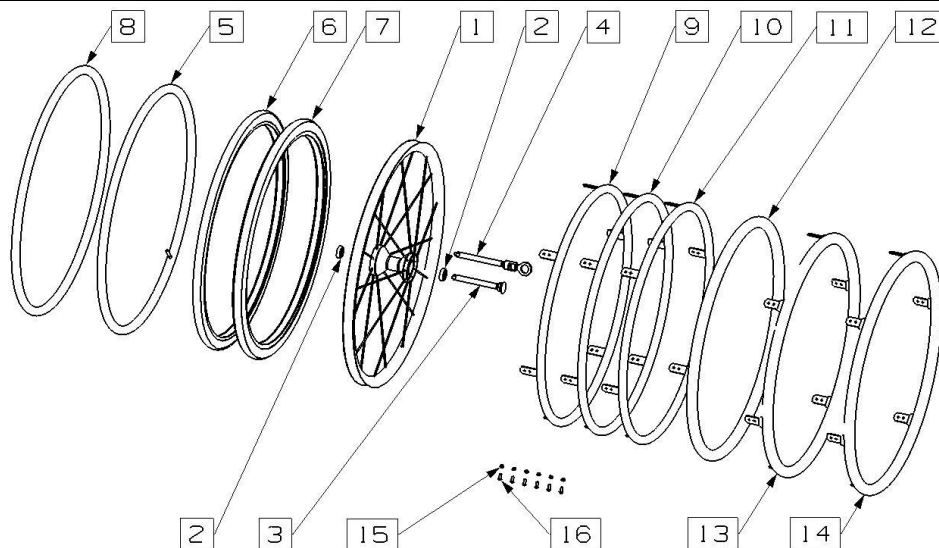

No.	Cat. No.	Element description	Qty
1	See options parts list	Spinergy L18 wheel 24"	2 X 1
2	See options parts list	Ball-bearing	2 X 2
3	See options parts list	Axle with cap	2 X 1
4	See options parts list	Axle with tetra handle	2 X 1
5	DETKA_24_S_1	Tube 24" for tyres	2 X 1
6	See options parts list	Schwalbe RightRun 24" tyre	2 X 1
7	OPONA_24_SM+_CZ	Schwalbe Marathon Plus 24" tyre	2 X 1
8	OPONA_24_TU_WS3_RR	Tufo 24" tubeless tyre	2 X 1
9	CALU_24	Aluminum pushrim 24"	2 X 1
10	CSTAL_24_GIETE_BL	Steel pushrim 24"	2 X 1
11	CTYT_24_GIETE_BL	Titanium pushrim 24"	2 X 1
12	NAKL_NA_C_24	Rubber cover for pushrims: aluminum, steel, titanium	2 X 1
13	CSTAL_24_POWL	Standard rubber covered pushrim 24"	2 X 1
14	CIAG_24_TETRAGRIP	Tetra silicone covered pushrim 24"	2 X 1
15	SRU_ISO7380_M5X10	Screw M5x10 for pushrim mounting	2 X 6
16	NAKR_SAMOHAM_M5	Self-locking nut M6 for pushrim mounting	2 X 6

Part options – Spinergy L18 wheels 18 spokes

No.	Cat. No.	Element description	Qty
1 / A	L.24.18M.111X2	Spinergy L18 wheel 24" with black spokes	2 X 1
1 / B	L.24.18M.133X2	Spinergy L18 wheel 24" with red spokes	2 X 1

Part options – ball bearings

No.	Cat. No.	Element description	Qty
2 / A	LOZ_CM_28,6X12_CX	Ball bearing for 12 mm axles	2 X 2
2 / B	31-0341	Ball bearing for 12,7 mm (1/2") axles	2 X 2

Part options – axles with caps

No.	Cat. No.	Element description	Qty
3 / A	OS_12X108_Z_KAP	12 mm axle with cap for standard radial wheels	2 X 1
3 / B	OS_12,7X109_Z_KAP	12,7 mm (1/2") axle with cap for standard radial wheels	2 X 1

Part options – axles with tetra handle


No.	Cat. No.	Element description	Qty
4 / A	OS_12X108_Z_KAP_Q	12 mm axle with tetra handle for standard radial wheels	2 X 1
4 / B	OS_12,8X108_Z_KAP_Q	12,7 mm (1/2") axle with tetra handle for standard radial wheels	2 X 1

Part options – Schwalbe RightRun tyres

No.	Cat. No.	Element description	Qty
6 / A	OPONA_24_SRR_SZ	Schwalbe RightRun 24" tyre, gray	2 X 1
6 / C	OPONA_24_SRR_CE	Schwalbe RightRun 24" tyre, red	2 X 1
6 / D	OPONA_24_SRR_N	Schwalbe RightRun 24" tyre, blue	2 X 1



Spinerger LX wheel 24" with 12 radial spokes

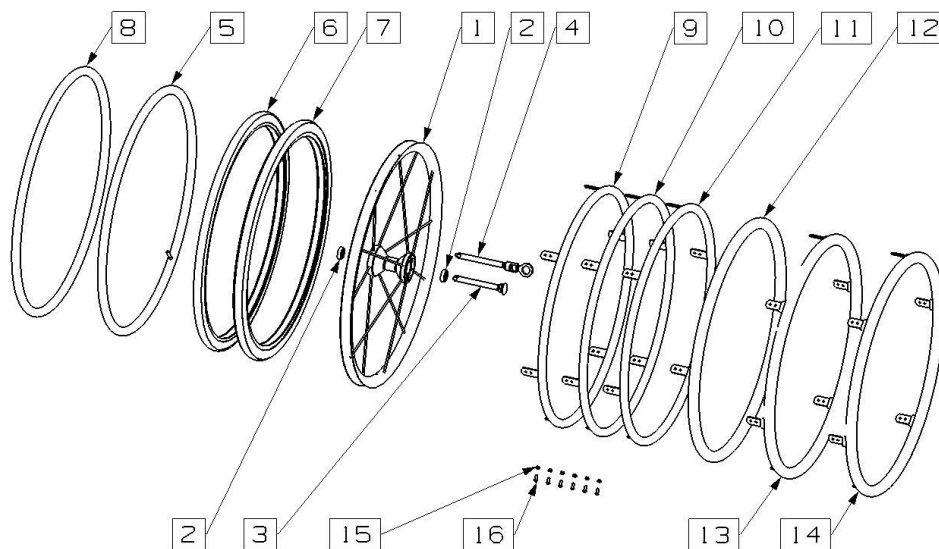

No.	Cat. No.	Element description	Qty
1	See options parts list	Spinerger LX12 wheel 24"	2 X 1
2	See options parts list	Ball-bearing	2 X 2
3	See options parts list	Axle with cap	2 X 1
4	See options parts list	Axle with tetra handle	2 X 1
5	DETKA_24_S_1	Tube 24" for tyres	2 X 1
6	See options parts list	Schwalbe RightRun 24" tyre	2 X 1
7	OPONA_24_SM+_CZ	Schwalbe Marathon Plus 24" tyre	2 X 1
8	OPONA_24_TU_WS3_RR	Tufo 24" tubeless tyre	2 X 1
9	CALU_24	Aluminum pushrim 24"	2 X 1
10	CSTAL_24_GIETE_BL	Steel pushrim 24"	2 X 1
11	CTYT_24_GIETE_BL	Titanium pushrim 24"	2 X 1
12	NAKL_NA_C_24	Rubber cover for pushrims: aluminum, steel, titanium	2 X 1
13	CSTAL_24_POWL	Standard rubber covered pushrim 24"	2 X 1
14	CIAG_24_TETRAGRIP	Tetra silicone covered pushrim 24"	2 X 1
15	SRU_ISO7380_M5X10	Screw M5x10 for pushrim mounting	2 X 6
16	NAKR_SAMOHAM_M5	Self-locking nut M6 for pushrim mounting	2 X 6

Part options – Spinerger L18 wheels 18 spokes

No.	Cat. No.	Element description	Qty
1 / A	LX.24.12M.111X2	Spinerger LX12 wheel 24" with black spokes	2 X 1
1 / B	LX.24.12M.133X2	Spinerger LX12 wheel 24" with red spokes	2 X 1

Part options – ball bearings

No.	Cat. No.	Element description	Qty
2 / A	LOZ_CM_28,6X12_CX	Ball bearing for 12 mm axles	2 X 2
2 / B	31-0341	Ball bearing for 12,7 mm (1/2") axles	2 X 2

Part options – axles with caps

No.	Cat. No.	Element description	Qty
3 / A	OS_12X108_Z_KAP	12 mm axle with cap for standard radial wheels	2 X 1
3 / B	OS_12,7X109_Z_KAP	12,7 mm (1/2") axle with cap for standard radial wheels	2 X 1

Part options – axles with tetra handle

No.	Cat. No.	Element description	Qty
4 / A	OS_12X108_Z_KAP_Q	12 mm axle with tetra handle for standard radial wheels	2 X 1
4 / B	OS_12,7X108_Z_KAP_Q	12,7 mm (1/2") axle with tetra handle for standard radial wheels	2 X 1


Part options – Schwalbe RightRun tyres

No.	Cat. No.	Element description	Qty
6 / A	OPONA_24_SRR_SZ	Schwalbe RightRun 24" tyre, gray	2 X 1
6 / C	OPONA_24_SRR_CE	Schwalbe RightRun 24" tyre, red	2 X 1
6 / D	OPONA_24_SRR_N	Schwalbe RightRun 24" tyre, blue	2 X 1

Spinerger SLX wheel 24" with 24 radial spokes



GTM-MOBIL M-T T. Walisiewicz Sp. Komandytowa • ul. Dorodna 16, 03-195 Warszawa • www.gtmmobil.com.pl
 service@gtmmobil.com.pl • foreign.office@gtmmobil.com.pl • tel./fax +48 022 504 12 75

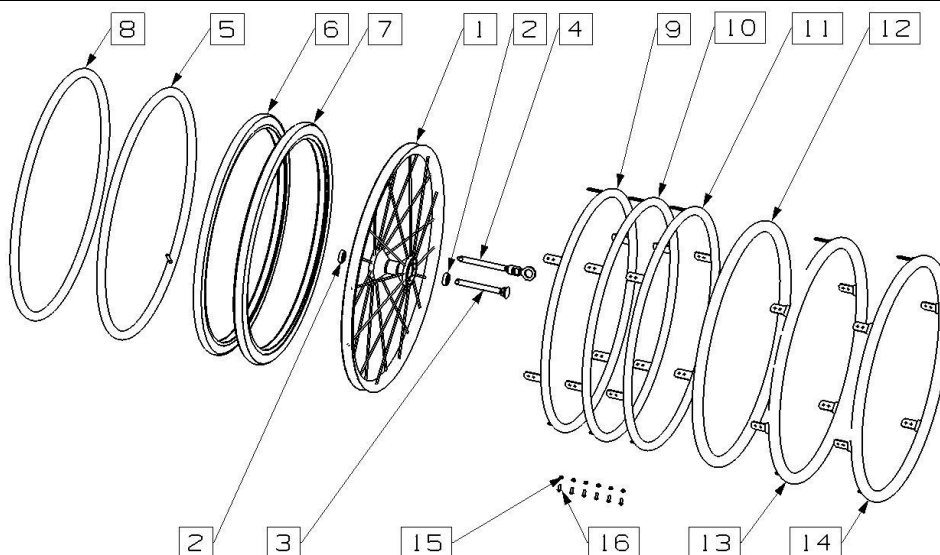

No.	Cat. No.	Element description	Qty
1	SLX.24.24M.111	Spinergy SLX wheel 24"	2 X 1
2	See options parts list	Ball-bearing	2 X 2
3	See options parts list	Axle with cap	2 X 1
4	See options parts list	Axle with tetra handle	2 X 1
5	DETKA_24_S_1	Tube 24" for tyres	2 X 1
6	See options parts list	Schwalbe RightRun 24" tyre	2 X 1
7	OPONA_24_SM+_CZ	Schwalbe Marathon Plus 24" tyre	2 X 1
8	OPONA_24_TU_WS3_RR	Tufo 24" tubeless tyre	2 X 1
9	CALU_24	Aluminum pushrim 24"	2 X 1
10	CSTAL_24_GIETE_BL	Steel pushrim 24"	2 X 1
11	31-0094	Titanium pushrim 24"	2 X 1
12	NAKL_NA_C_24	Rubber cover for pushrims: aluminum, steel, titanium	2 X 1
13	CSTAL_24_POWL	Standard rubber covered pushrim 24"	2 X 1
14	CIAG_24_TETRAGRIP	Tetra silicone covered pushrim 24"	2 X 1
15	SRU_ISO7380_M5X10	Screw M5x10 for pushrim mounting	2 X 6
16	NAKR_SAMOHAM_M5	Self-locking nut M6 for pushrim mounting	2 X 6

Part options – ball bearings

No.	Cat. No.	Element description	Qty
2 / A	LOZ_CM_28,6X12_CX	Ball bearing for 12 mm axles	2 X 2
2 / B	31-0341	Ball bearing for 12,7 mm (1/2") axles	2 X 2

Part options – axles with caps

No.	Cat. No.	Element description	Qty
3 / A	OS_12X108_Z_KAP	12 mm axle with cap for standard radial wheels	2 X 1
3 / B	OS_12,7X109_Z_KAP	12,7 mm (1/2") axle with cap for standard radial wheels	2 X 1

Part options – axles with tetra handle


No.	Cat. No.	Element description	Qty
4 / A	OS_12X108_Z_KAP_Q	12 mm axle with tetra handle for standard radial wheels	2 X 1
4 / B	OS_12,8X108_Z_KAP_Q	12,7 mm (1/2") axle with tetra handle for standard radial wheels	2 X 1

Part options – Schwalbe RightRun tyres

No.	Cat. No.	Element description	Qty
6 / A	OPONA_24_SRR_SZ	Schwalbe RightRun 24" tyre, gray	2 X 1
6 / C	OPONA_24_SRR_CE	Schwalbe RightRun 24" tyre, red	2 X 1
6 / D	OPONA_24_SRR_N	Schwalbe RightRun 24" tyre, blue	2 X 1



Standard wheel 26" with radial spokes

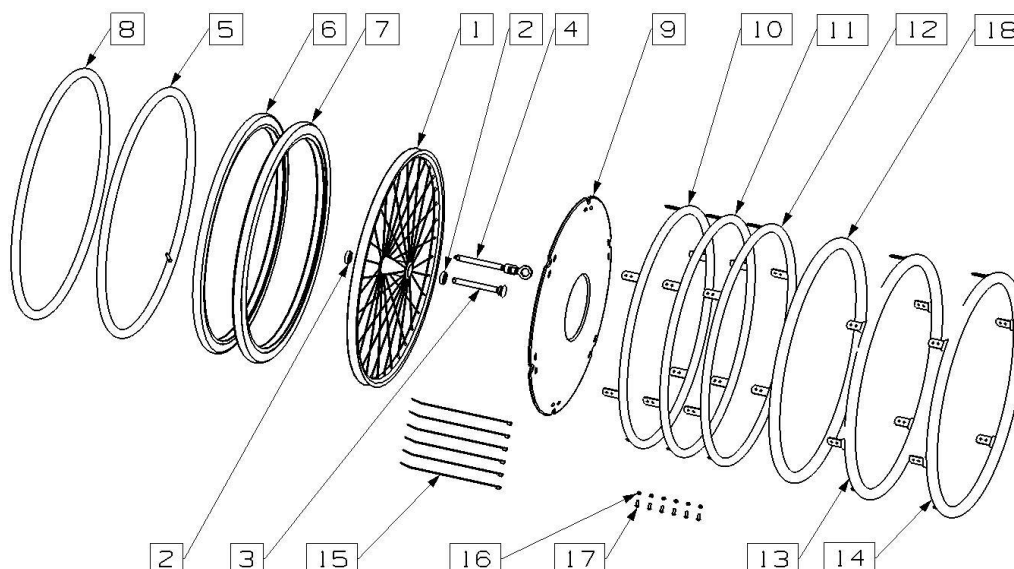

No.	Cat. No.	Element description	Qty
1	-	Standard radial wheel 26"	2 X 1
2	See options parts list	Ball-bearing	2 X 2
3	See options parts list	Axle with cap	2 X 1
4	See options parts list	Axle with tetra handle	2 X 1
5	DETKA_24_S_1	Tube 24" for tyres	2 X 1
6	See options parts list	Schwalbe RightRun 24" tyre	2 X 1
7	OPONA_24_SM+_CZ	Schwalbe Marathon Plus 24" tyre	2 X 1
8	OPONA_24_TU_WS3_RR	Tufo 24" tubeless tyre	2 X 1
9	See options parts list	Spokesguard 24"	2 X 1
10	CALU_24	Aluminum pushrim 24"	2 X 1
11	CSTAL_24	Steel pushrim 24"	2 X 1
12	CTYT_24_352485	Titanium pushrim 24"	2 X 1
13	CSTAL_24_POWL	Standard rubber covered pushrim 24"	2 X 1
14	CIAG_24_TETRAGRIP	Tetra silicone covered pushrim 24"	2 X 1
15	OP_ZAC_250/4,8_BIA	Spokesguard connector	2 X 6
16	SRU_ISO7380_M5X10	Screw M5x10 for pushrim mounting	2 X 6
17	NAKR_SAMOHAM_M5	Self-locking nut M6 for pushrim mounting	2 X 6
18	NAKLAD_NA_C_24	Rubber cover for pushrims: aluminum, steel, titanium	2 X 1

Part options – ball bearings

No.	Cat. No.	Element description	Qty
2 / A	LOZ_28-12-8-6001CX	Ball bearing for 12 mm axles	2 X 2
2 / B	LOZ_MC_28X12,8_CX	Ball bearing for 12,7 mm (1/2") axles	2 X 2

Part options – axles with and without caps

No.	Cat. No.	Element description	Qty
3 / A	OS_12X98_B_KAP	12 mm axle with cap for standard radial wheels	2 X 1
3 / B	OS_12,8X98_B_KAP	12,7 mm (1/2") axle without cap for standard radial wheels	2 X 1

Part options – axles with tetra handle

No.	Cat. No.	Element description	Qty
4 / A	OS_12X98_Z_KAP_Q	12 mm axle with tetra handle for standard radial wheels	2 X 1
4 / B	OS_12,8X98_Z_KAP_Q	12,7 mm (1/2") axle with tetra handle for standard radial wheels	2 X 1

Part options – Schwalbe RightRun tyres


No.	Cat. No.	Element description	Qty
6 / A	OPONA_24_SRR_SZ	Schwalbe RightRun 24" tyre, gray	2 X 1
6 / C	OPONA_24_SRR_CE	Schwalbe RightRun 24" tyre, red	2 X 1
6 / D	OPONA_24_SRR_N	Schwalbe RightRun 24" tyre, blue	2 X 1

Part options – spokesguards

No.	Cat. No.	Element description	Qty
9 / A	OSL_SPR_24_BBARW	Spokesguard 24", transparent	2 X 1
9 / B	OSL_SPR_24_GTM	Spokesguard 24", transparent with GTM Logo	2 X 1



Sport wheel 26" with crossed spokes

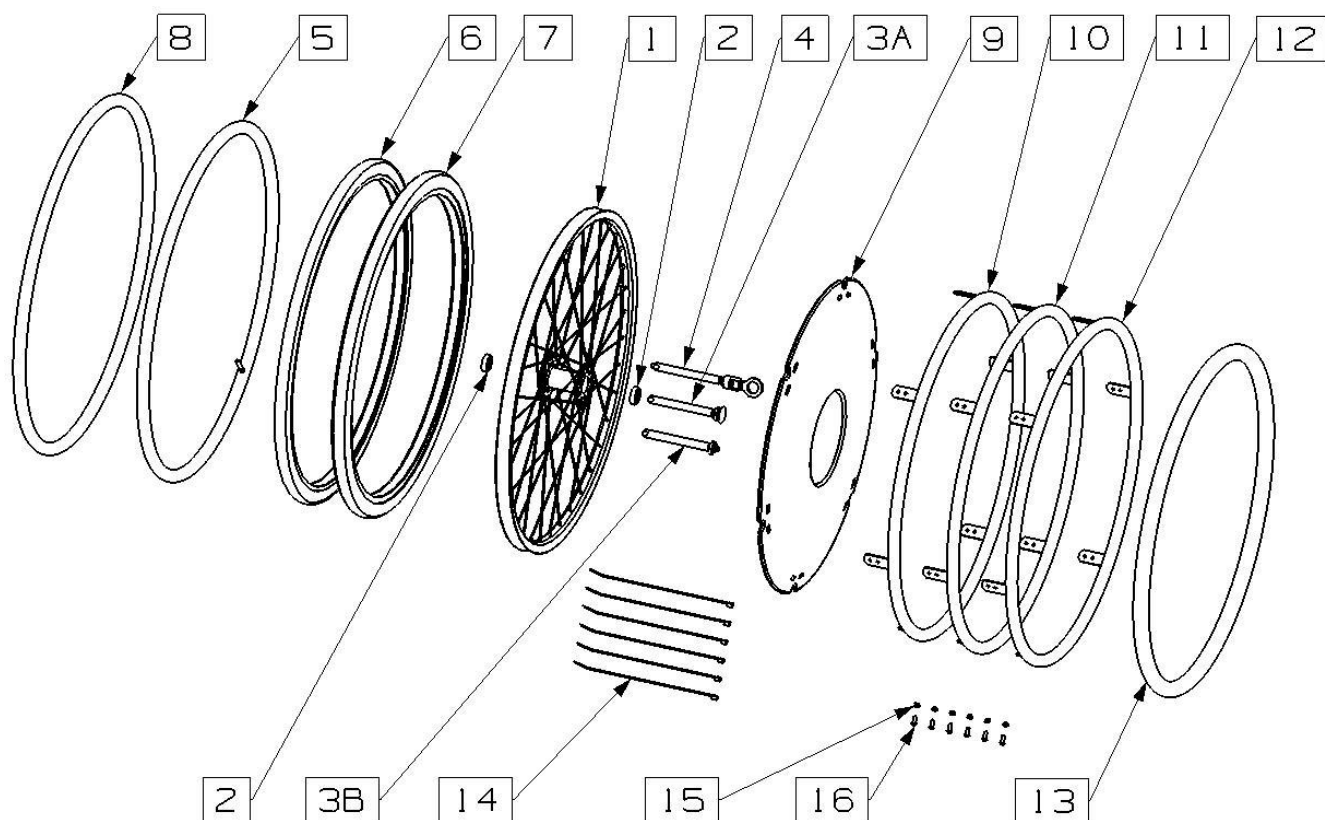

No.	Cat. No.	Element description	Qty
1	Individual part	Sport crossed wheel 26"	2 X 1
2	See options parts list	Ball-bearing	2 X 2
3A	OS_12X103_Z_KAP	Axle 12 mm with cap	2 X 1
3B	OS_12,8X104_B_KAP	Axle 12,7 mm (1/2") without cap	2 X 1
4	See options parts list	Axle with tetra handle	2 X 1
5	DETKA_26_S	Tube 26" for tyres	2 X 1
6	OPONA_26_S_SZ	Schwalbe RightRun 26" tyre, gray	2 X 1
7	OPONA_26_SM+E	Schwalbe Marathon Plus 26" tyre	2 X 1
8	OPONA_MS3_TU	Tufo 26" tubeless tyre	2 X 1
9	OSL_SPR_26_BBARW	Spokesguard 26", transparent	2 X 1
10	CALU_26	Aluminum pushrim 26"	2 X 1
11	CSTAL_26	Steel pushrim 26"	2 X 1
12	CTYT_26	Titanium pushrim 26"	2 X 1
13	NAKLAD_NA_C_26	Rubber cover for pushrims: aluminum, steel, titanium	2 X 1
14	OP_ZAC_250/4,8_BIA	Spokesguard connector	2 X 6
15	SRU_ISO7380_M5X10	Screw M5x10 for pushrim mounting	2 X 6
16	NAKR_SAMOHAM_M5	Self-locking nut M6 for pushrim mounting	2 X 6

Part options – ball bearings


No.	Cat. No.	Element description	Qty
2 / A	LOZ_28-12-8_6001	Ball bearing for 12 mm axles	2 X 2
2 / B	LOZ_MC_28X12,8_CX	Ball bearing for 12,7 mm (1/2") axles	2 X 2

Part options – axles with tetra handle

No.	Cat. No.	Element description	Qty
4 / A	OS12X104_Z_KAP_Q	12 mm axle with tetra handle for standard radial wheels	2 X 1
4 / B	OS_12,8X104_Z_KAP_Q	12,7 mm (1/2") axle with tetra handle for standard radial wheels	2 X 1



Spinergy SLX wheel 26" with 24 radial spokes

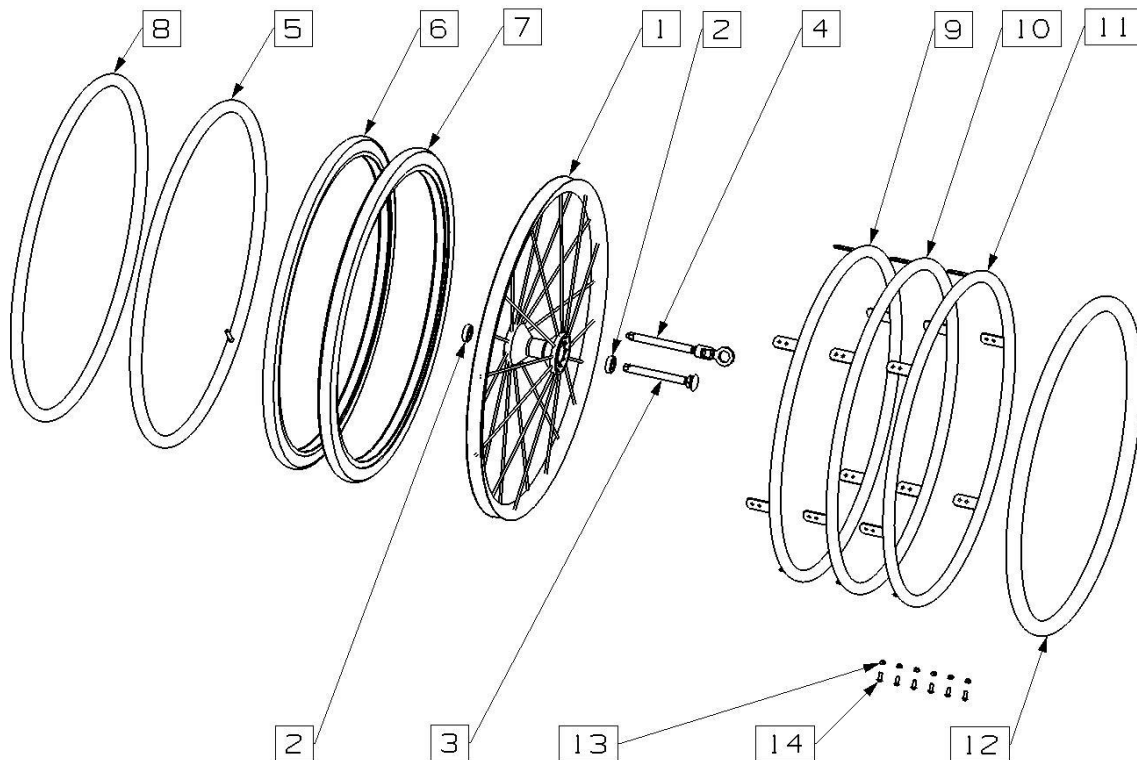

No.	Cat. No.	Element description	Qty
1	SLX.26.24M.111	Spinergy SLX24 wheel 26" with black spokes	2 X 1
2	See options parts list	Ball-bearing	2 X 2
3	See options parts list	Axle with cap	2 X 1
4	See options parts list	Axle with tetra handle	2 X 1
5	DETKA_26_S	Tube 26" for tyres	2 X 1
6	OPONA_26_S_SZ	Schwalbe RightRun 26" tyre, gray	2 X 1
7	OPONA_26_SM+E	Schwalbe Marathon Plus 26" tyre	2 X 1
8	OPONA_26_MS3_TU	Tufo 26" tubeless tyre	2 X 1
9	CALU_26	Aluminum pushrim 26"	2 X 1
10	CSTAL_26_GIETE_BL	Steel pushrim 26"	2 X 1
11	CTYT_26	Titanium pushrim 26"	2 X 1
12	NAKL_NA_C_26	Rubber cover for pushrims: aluminum, steel, titanium	2 X 1
13	NAKL_ISO7380_M5X10	Screw M5x10 for pushrim mounting	2 X 6
14	NAKR_SAMOHAM_M5	Self-locking nut M6 for pushrim mounting	2 X 6

Part options – ball bearings

No.	Cat. No.	Element description	Qty
2 / A	LOZ_CM_28,6X12_CX	Ball bearing for 12 mm axles	2 X 2
2 / B	LOZ_CC_28,6X12,8_CX	Ball bearing for 12,7 mm (1/2") axles	2 X 2

Part options – axles with caps


No.	Cat. No.	Element description	Qty
3 / A	OS_12X108_Z_KAP	12 mm axle with cap for Spinergy wheels	2 X 1
3 / B	OS_12,7X109_Z_KAP	12,7 mm (1/2") axle with cap for Spinergy wheels	2 X 1

Part options – axles with tetra handle

No.	Cat. No.	Element description	Qty
4 / A	OS_12X108_Z_KAP_Q	12 mm axle with tetra handle for Spinergy wheels	2 X 1
4 / B	OS_12,8X108_Z_KAP_Q	12,7 mm (1/2") axle with tetra handle for Spinergy wheels	2 X 1



Antitipper wheel

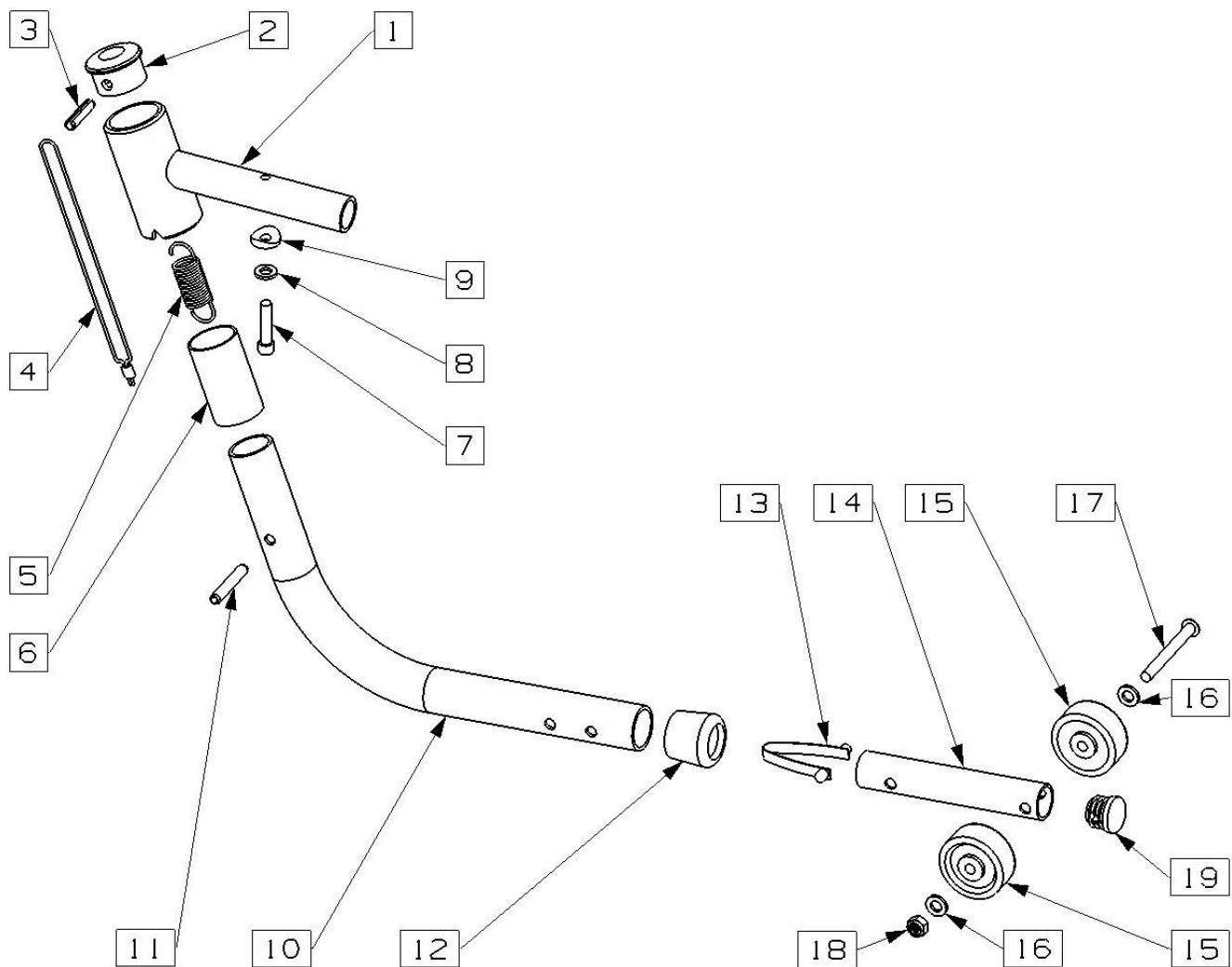

No.	Cat. No.	Element description	Qty
1	11-0801	Antitipper anchorage	2 X 1
2	11-0630	Anchorage cap	2 X 1
3	KOLEK_SPREZYSTY_6X26	Spring pin 6 x 26	2 X 1
4	LINKA_STAL_FI_1,5MM	Spring mounting link	2 X 1
5	31-0002	Spring	2 X 1
6	TULEJA_WASA	Plastic sleeve	2 X 1
7	SRU_DIN912_M6X25	Screw M6x25	2 X 1
8	PODKL_DIN_125_A2_M6	Steel washer M6	2 X 1
9	11-0297	Saddle washer	2 X 1
10	11-0366	Antitipper tube	2 X 1
11	KOLEK_SPREZYSTY_6X36	Spring pin 6 x 36	2 X 1
12	11-0195	Plastic flange	2 X 1
13	SPR_PLASKA	Flat spring	2 X 1
14	11-0372	Telescope tube	2 X 1
15	K_WAS	Plastic castor	2 X 2
16	PODKL_M6	Steel washer M6	2 X 2
17	SRU_7380_M6X65_INOX	Screw M6 x 65	2 X 1
18	NAKR_SAMOHAM_M6_CX	Self-locking nut M6	2 X 1
19	KOR_PLAS_R20	Plastic cap	2 X 1

