# VERMEIREN

## V-drive

INSTRUCTION MANUAL - MODE D'EMPLOI - GEBRUIKSAANWIJZING  
GEBRAUCHSANWEISUNG - ISTRUZIONI PER L'USO - MANUAL DE INSTRUCCIONES

**Notice to the specialist dealer:**

This instruction manual is part and parcel of the product and it should accompany each product delivered.

1. Edition 2010

All rights reserved, including translation.

No part of this manual may be reproduced in any form whatsoever (print, photocopy, microfilm or any other process) without written permission of the publisher, or processed, duplicated, or distributed by using electronic systems.

© Vermeiren 2010

## CONTENTS

1. Introduction
2. Safety Procedures
3. Parts Description
4. Installation Instructions
5. Disassembly and Transportation
6. Battery Charging and Care
7. Operating the V-Drive
8. Troubleshooting
9. Specifications
10. Warranty
11. Compatibility with Drive Wheelchairs

## INTRODUCTION

The V-Drive is designed as an aid to a wheelchair attendant providing power to aid pushing in otherwise strenuous situations such as up steep hills, etc.

The V-Drive features some ingenious concepts to help the installation and operation of the device. Some these are covered by a pending patent GB 0723584.9

The V-Drive is designed to work in conjunction with most wheelchairs with seat widths between 16" and 20" (39 cm – 48 cm). An optional extension bar is available to fit to wheelchairs with 22" (55 cm) seat widths.

Dual-wheel and Heavy Duty versions of the V-Drive are available.

The V-Drive is to be controlled by an attendant with a manual wheelchair and user up to a combined weight of 135kg (\*H.D.: 200 kg). The V-Drive uses a 24V 12Ah (\*H.D.: 12Ah) battery pack, and has a range of up to nine miles between charges.

(\*H.D.: Heavy Duty version)

## SAFETY PROCEDURES

The V-Drive must not be used solely by the wheelchair user under any circumstances.

Ensure the V-Drive is installed as far to the rear of the wheelchair as is comfortable for the attendant to do so.

Operate the V-Drive at a speed that is both comfortable for the operator and wheelchair user. When connecting / disconnecting the battery from the motor or the charger make sure both the power to the charger and V-Drive handcontrol are switched off.

Ensure the V-Drive is secured in transit. It is also recommended that the battery is disconnected. Ensure that the battery is charged only with the battery charger supplied with the V-Drive. Using another battery charger may cause damage or injury.


Whilst the V-Drive has been EMC tested, it may be susceptible to certain types of electromagnetic waves. Avoid close proximity to mobile phones, commercial broadcast towers and emergency services' radios.

Other household appliances are not known to pose any such risk. If involuntary movement occurs, switch off of the V-Drive and report this to your dealer.

The V-Drive should not be used when it is snowing or during heavy rain. This may cause damage to the V-Drive. The V-Drive should not be exposed to damp conditions or direct sunlight.

**Note: Wheelchairs fitted with attendant cable brakes.** On such wheelchairs, the attendant brake levers should be removed prior to fitting. If the attendant brakes do not have a separate parking brake facility, then parking brakes should be added to the wheelchair.


## PARTS DESCRIPTION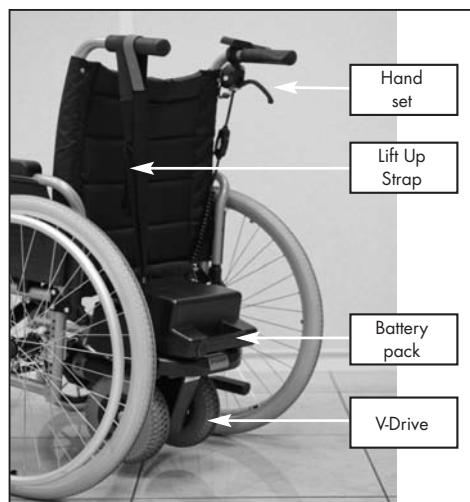

## INSTALLATION INSTRUCTIONS

### 1. Handset Installation

Push the controller forwards to remove it from the clamp. Loosen both screws of the clamp and position it around the tube of your wheelchair handle (You can use the circular rubber if the wheelchair tube is too small).

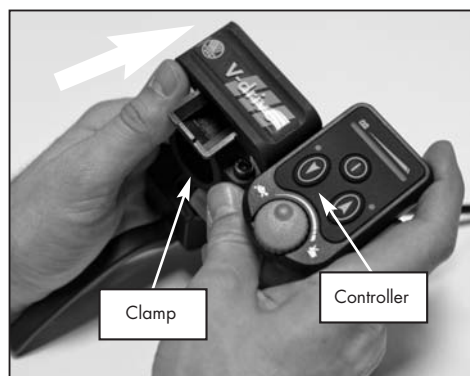Use an Allen key to tighten the clamp around the wheelchair handle. Slide the controller back over the clamp.

### 2. Bracket Installation

Install both brackets on the bottom horizontal tubing of the wheelchair, using an Allen Key. Attach the two sections of Velcro together. Ensure that the screws are sufficiently tight and do not rotate on the tubing.

### 3. V-Drive Installation

Feed the telescopic bar through the front end of the V-drive. Loosen the tristar knob on the bar and adjust to the correct length.

Tighten the knobs on the bar and at the front of the V-drive. Place the two ends of the bar on the rear of the brackets, then push the V-drive forward until it locks in position.

### 4. Battery Pack Installation

Place the battery pack in to the battery tray as

shown in the photograph. Secure the battery pack by pushing it all the way in. An audible click will be heard.

### 5. Final Connection

Connect the cable from the V-drive in to the underneath of the handset.

Place the top loop of the lift-up strap around the opposite push handle to the handset.

### DISASSEMBLY AND TRANSPORTATION

1. Switch the V-Drive off at the ignition.
2. Disconnect / unclip the V-drive connector from underneath the handset.
3. Remove the lift-up strap from the push handle.

4. Lift the battery pack out of the battery tray.
5. Remove the battery pack out of the tray by pulling at the handle of the battery pack and simultaneously pushing the PUSH button at the bottom of the battery pack.
6. Place all items in the carry holdall supplied.

**Notes:** The battery supplied with the V-Drive is classified as safe for air travel by compliance with IATA special provision A67. The battery pack and V-drive have fabric handles to make them easier to carry.

## BATTERY CHARGING AND CARE

Follow the procedure below step by step:

1. Turn the power switch to (OFF).
2. Lift out the battery pack using the handle on the pack.
3. Plug the charger's power cord into the mains.
4. Plug the charger's round plug into the charging socket of the battery pack
5. Switch on the plug socket.
6. The charger's LED will illuminate orange to indicate the batteries are charging. The charging duration is about 8 hours. To ensure optimum performance a 12-hour charge is recommended.
7. The orange LED will turn green when charging is complete.
8. Turn off the charger, disconnect the power cord and the round plug from the battery pack.

### Suggestions

1. Do not disconnect the charger cord if charging is not completed. The battery life will be seriously shortened or decayed if the battery is repeatedly used without being fully charged. Therefore be sure to charge the battery fully every time.
2. Always complete the charging until the LED light turns green. NEVER stop charging before it is complete.
3. When fully charged, the battery charger will still trickle charge the battery to ensure optimum range.
4. If you do not use your powerpack for a long time, it should be charged at least every week to keep the battery in a fully usable condition.
5. The ambient temperature will affect charging time. Charging time will be longer in the winter.
6. After charging, do not leave the charger socket plugged in to the scooter, as this will cause a power drain on the powerpack and temporarily reduce its range.

7. The batteries carry a six month manufacturer's warranty. This warranty only covers issues relating to manufacturing faults, and not faults relating to failure to recharge the batteries as instructed above.

**Attention:** Please follow the rules below to avoid accidents while charging.

1. Only use the supplied charger, and recharge the battery to its full capacity every time. You may damage the battery if you use a charger which is not to the correct specification.
2. Never disassemble or modify the charger.
3. Please charge in a well-ventilated space where there is no direct exposure to the sunlight. Do not charge in surroundings where it is humid, raining or dewy.
4. Do not charge in temperatures less than -10°C or higher than +50°C as the charger may not work well & batteries may become damaged.

### Warnings

1. Keep away from flammable objects while charging as this may lead to fire or battery explosion.
2. Do not smoke while charging as the battery may release hydrogen gas. Always charge your battery in a well-ventilated space.
3. Never connect or disconnect the plug or cord with wet hands whilst charging. Do not connect or disconnect the plug or cord when they are wet, it may lead to electric shock.

### Charger

The charger supplied has an LED which changes colour depending on its status:

If the LED is lit ORANGE then the charger is switched on and charging

If the LED is lit GREEN then the charger is switched on and has finished the bulk of its charging and the batteries can be used. However, we recommend that you leave the charger charging for a couple of hours after the green LED has lit. This will maintain and increase the capacity of your batteries.

The LED will also illuminate GREEN if the charger is switched on but not connected to the batteries.

## Batteries

Do not expose the battery pack to temperatures below  $-10^{\circ}$  or above  $50^{\circ}$  C when charging or storing.

The batteries are maintenance free and there is no need to inspect the battery liquid or refill with water.

You are required to charge the batteries on a regular basis. Even if the powerpack is not used, it should be charged at least once a week.

The battery pack contains two 12V 12Ah (H.D.: 12Ah) batteries.

The battery supplied with the V-Drive is classified as safe for air travel by compliance with IATA special provision A67.

## Charging the Batteries

Using the charger supplied, the charging duration is about 8 hours. However, to ensure optimum performance we recommend a 12-hour charge.

We also recommend that the batteries are not charged for more than 24 hours.

## OPERATING THE V-DRIVE

*Direction.* Press the power button. The light of the forward button will light up to indicate forward movement. Press the backward button to make the V-drive move backwards.


*Lift-up strap.* The V-drive comes with a lift-up strap to allow the wheelchair to be pushed manually. Lift up the V-drive with the red strap and hook the second loop around the push handle of the wheelchair.

*Maximum Speed.* Always set a maximum speed which you are comfortable walking with.

*Pushing the wheelchair when not in use.* We recommend that the powerpack is restrained with the lift-up strap when not in use.

## HANDSET OPERATION


Please note there is a 2-second delay when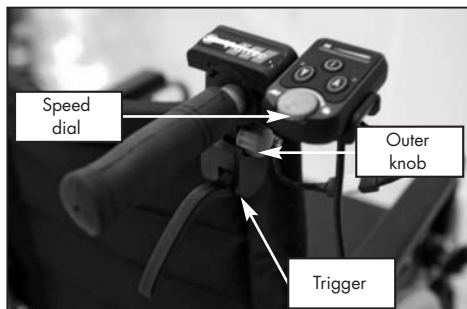

starting up the V-Drive before the wheel will move. Wait 2 seconds before squeezing the trigger.

**Trigger** – Squeeze the trigger to propel the V-Drive. The more the trigger is squeezed, the faster the V-Drive will travel. Release the trigger to stop the V-Drive.

**Speed dial** – Turn the dial clockwise to increase the maximum speed of the V-Drive. Turn the dial anticlockwise to reduce the V-Drive's maximum speed.

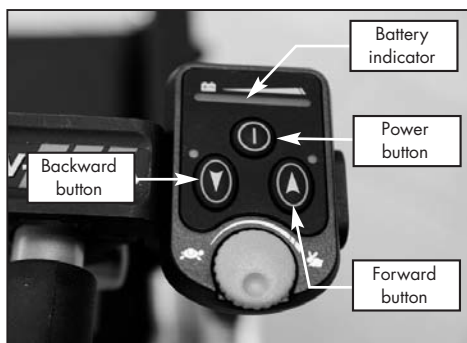Outer Knob – This is used to determine the angle of the handset.

## SPECIFICATIONS

Weight: 11.8kg (dual wheel w/o battery)

9 kg (battery pack, 2x12Ah)

Max speed: 4mph / 6,4 km/h

Max gradient: 8°

Cruising range up to 10 miles (16 km)

Weight capacity: 135kg including wheelchair & user (H.D.: 200 kg)

Motor: 24V 150W D.C. (H.D.: 200W)

Battery: 2x 12V 12Ah (H.D.: 12 Ah)

Charger: 2A 24V output

Controller: PG - S - DRIVE 45A

## TROUBLESHOOTING

*Wheelchair does not move when trigger is pressed.*

Possible Solutions:

- Check the V-Drive is switched on
- Check the battery is fully charged.
- Check the connection between the battery and motor is correct.
- Check the wheelchair brakes are not applied.
- Check the drive wheel is in contact with the ground.

*Wheelchair is difficult to push manually.*

Possible Solutions:

- Check the wheelchair brakes are not applied.
- Remove V-Drive.

*The V-Drive is too fast/slow*

- Try varying the speed with speed dial.
- Try recharging the batteries (if too slow).
- Check the wheelchair brakes are not applied.

## WARRANTY

To improve the safety and lifespan of your V-Drive, it is recommended that you have it serviced on a regular basis. Recommended Service Schedules for V-Drives are available to service agents.

There is a comprehensive 24-month warranty from the date on which your new V-Drive is delivered. The warranty covers the V-Drive for construction errors during this period. For more detail, please see the warranty conditions below:

1. Any work or replacement part installation must be carried out by an authorised service agent.
  2. To apply the warranty should your V-Drive require attention please contact the outlet from which you purchased the V-Drive.
  3. Should any part of the V-Drive require repair or full or part replacement, as a result of a manufacturing or material defect within the warranty period, parts will be supplied free of charge.
- Note:** The guarantee is not transferable.
4. Any repaired or replaced parts will be covered by the balance of the warranty period on the V-Drive.
  5. Parts replaced after the original warranty has expired will be covered by a three-month warranty.
  6. Consumable items supplied will not gene-

rally be covered during the normal warranty period unless such items require repair or replacement clearly as a direct result of a manufacturing or material defect. Such items include (among others): upholstery and tyres.

7. The above warranty conditions apply to a brand new V-Drives. Under normal circumstances, no responsibility will be accepted where the V-Drive has required assistance as a direct result of:

- a. The V-Drive part not having been maintained in accordance with the manufacturers recommendation.
- b. Failure to use the manufacturer's specified parts.
- c. The V-Drive or part having been damaged due to neglect accident or improper use.
- d. The V-Drive or part having been altered from the manufacturer's specification or repairs having been attempted before the dealer is notified.


In the event of your V-Drive requiring attention, please contact your service agent/dealer and give all relevant details so they can act quickly.

The manufacturer reserves the right to alter without notice any weights, measurements or other technical data shown in this manual. All figures, measurements and capacities shown in this manual are approximate and do not constitute specifications.

## COMPATIBILITY WITH WHEELCHAIRS

The V-Drive is suitable for most folding wheelchairs:

- With a distance between tubing edges (W) of 14½" to 20" (37cm – 50cm). This is usually, but not always, the same as the wheelchair seat width
- A round tubing diameter (D) between 7/8" - 1" (22mm – 25mm)

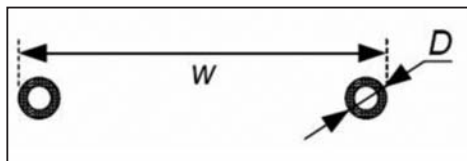Please note that you should check with your dealer and/or the wheelchair manufacturer. Vermeiren N.V. does not accept any responsibility for other manufacturer's wheelchairs, and in some circumstances a risk assessment should be carried out prior to installation.

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### **Vermeiren Suisse S.A.**
### **Vermeiren Nederland B.V.**