

Vector

SPARE PARTS LIST



[SORG Rollstuhltechnik](#)

Revision: 20.08.2018 09:45:54



Dear customer,
in the following pages you will find a complete spare part list with all parts used to build our wheelchairs.

To ensure a accurate delivery of spare parts it is vital to provide the following information fro the identification label in addition to the Article-Number (Art.-No.).

1. Serial number of the chair

The serial number enables us to identify the configuration of delivery of the specific chair in.

2. Seat width, seat depth and back height

The measures on the identification label are specified for the delivery configuration. If you changed any of these measures after delivery e.g. by growing the chair, please specify the actual new measures.

3. Lower leg length

We need the LLL to decide for the right version of several attachments which are connected to the LLL.

4. Frame colour

Please indicate the frame colour of the chair when ordering a powder coated spare part. Please note also if the finish alternative is shiny or matt.

5. Any other change after delivery (e.g. attachments)

We cannot know about changes after delivery. If you are not sure if such a change has any influence on the needed spare parts, please give us some details about it.

List of abbreviations

Pcs.	piece number
item no	item number to order specific spare part
Q/1	quantity per wheelchair/assembly group
SW	seat width
SD	seat depth
BH	back height
alt.	alternative
l/r	left/right

Handling annotations

Only items with item number are to order.
right / left are in relation to driving direction.

If you need further assistance, please contact us:

SORG Rollstuhltechnik GmbH & Co.KG

Benzstraße 3-5
68794 Oberhausen-Rheinhausen
Fon +49 7254 – 9279 0
Fax +49 7254 – 9279 10

info@sorgrollstuhltechnik.de

<https://www.sorgrollstuhltechnik.de>

Opening times:
Monday – Friday 8.00 am – 4.30 pm

Technical alterations and misprints excluded.
Prices on demand.

Contents

Frame

Frame	1
Frame XL	3
Frame extension for seat shell	4
Anti-tipper	6
Tipping aid	7
Crutch holder with tipping aid	8

Leg support

Leg support standard, internal mounting 403901	9
Leg support standard, external mounting 404001	10
Leg support undivided, hinged sideways, internal mounting, (version from July 2013)	11
Leg support undivided, hinged sideways, internal mounting, (version until June 2013)	14
Leg support undivided, hinged sideways, external mounting, (version from July 2013)	15
Leg support undivided, hinged sideways, external mounting, (version until June 2013)	18
Leg support undivided, hinged backwards, internal mounting, N° 403903	20
Leg support undivided, hinged backwards, external mounting, N° 403904 (version from September 2013)	22
Leg support undivided, hinged backwards, external mounting, N° 403904 (version until August 2013)	24
Leg support undivided, hinged sideways and turnable outwards (version from May 2013)	26
Leg support undivided, hinged sideways and turnable outwards (version until April 2013)	29
Foot rest undivided	31
Leg support divided, hinged sideways, internal mounting 403602	32
Leg support divided, hinged sideways, external mounting 404003	33
Heel cup	34
Calf belts	35

Seat

Seat cover + cover carrier	36
SeatFix + SeatFix carrier	37
Seat cushion	39
Abduction wedge	40
Therapy table, version 1, hinged sideways	41
Therapy table, version 2, fixed to the arm pad	42
Therapy table, version 6, mounted under seat plate	43

Contents

Back

Back tube standard	44
Back, adjustable in angle	46
Back rest with gas pressure spring adjustment	48
Stabilizer bar, tripartite	49
Stabilizer bar, undivided	50
Push handles, extendable	52
Push bar, extendable from (SW32)	54
Push bar, extendable (to SW30)	55
Seat cover, adjustable	56
Curved back plate	57
Back elongation	60
Cushion for curved back plate	61
Truss pads Stealth from BH 35cm	62
Head rest	64

Side part

Side guards standard (from 10.06.2013)	66
Side guards standard (until 09.06.2013)	68
Arm rest for side guards standard (from 10.06.2013)	69
Arm rest for side guards standard (until 09.06.2013)	70
Clothes guard side guard (from 10.06.2013)	71
Clothes guard side guard (until 09.06.2013)	72
Arm rest for clothes guard (until 10.06.2013)	73
Wheel cover (until 10.06.2013)	74
Desk side guard	75
Clothes guard for desk side guards	77
Arm rest for desk side guards	78
Long side guard	79
Arm rest for long side guards	81
Desk/long side guard, front socket	82
Arm pad with raster	83
Side guard for back with gas pressure spring	84
Desk side guard for back with gas pressure spring	85
Long side guard for back with gas pressure spring	86

Caster wheel

Caster wheel adapter Aluminium	88
Adapter for low seat heights	90
Caster wheel fork	91
Caster wheel fork with quick-release axle	93
Caster wheels	94



Contents

Rear wheel

Axle plate standard	95
Axle plate drum brake	97
Wheel base lengthening	98
Rear wheels	100
Double handrim standard	101
Double handrim drum brake	102
Telescopic rod	104
Tyres	105
Handrims	106
Handrim covers	107
Spoke covers	108


Brake

Knee lever brake standard	109
Knee lever brake pull-to-lock	110
Knee lever brake lockable	111
Knee lever brake for attendant	112
One-hand brake operating side	114
One-hand brake non-operating side	116
Bowden cable brake pressure piece	118
Bowden cable brake control/non-control sides	120
Brake lever for drum brake	121
Brake lever extensions	122

Options

Caddy	123
Water bottle	124
Hip belt with Velcro and synthetic lock	125
Hip belt with metal lock	126
Butterfly harness	127

Frame

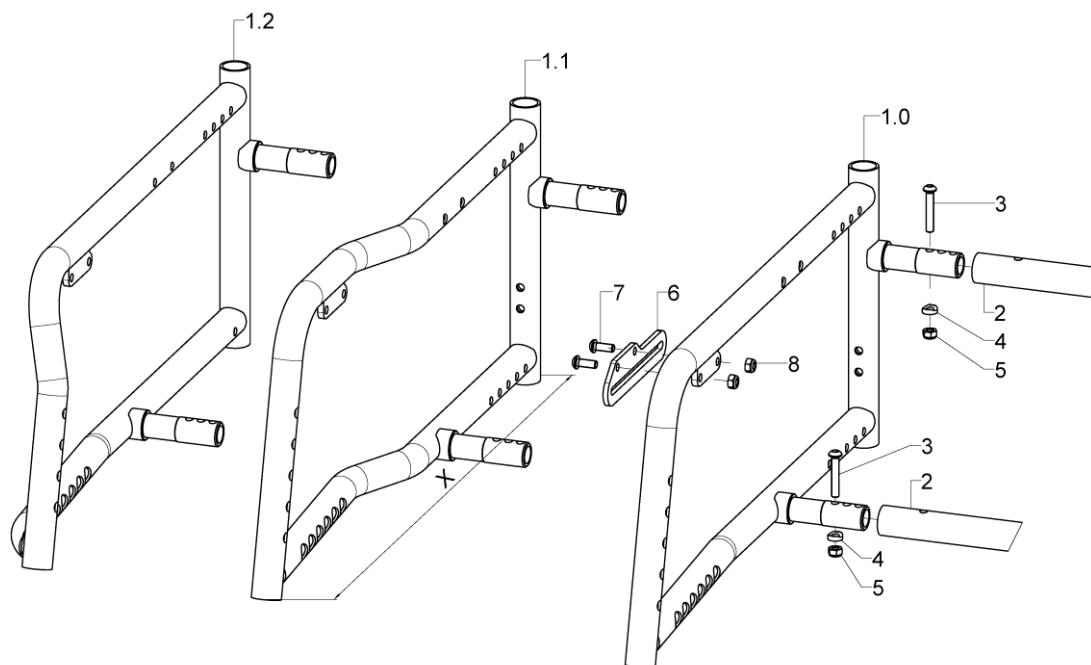

pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1.0	1	59690502	frame part straight, right, FS 1		59.69.05.00.02.69
o. B.	1	59690501	frame part straight, left, FS 1		59.69.05.00.01.69
1.0	1	59690602	frame part straight, right, FS 2		59.69.06.00.02.69
o. B.	1	59690601	frame part straight, left, FS 2		59.69.06.00.01.69
1.0	1	59690702	frame part straight, right, FS 3		59.69.07.00.02.69
o. B.	1	59690701	frame part straight, left, FS 3		59.69.07.00.01.69
1.0	1	59690802	frame part straight, right, FS 4		59.69.08.00.02.69
o. B.	1	59690801	frame part straight, left, FS 4		59.69.08.00.01.69
1.0	1	59691201	frame part straight, right, FS 5		59.69.16.00.02.69
o. B.	1	59691202	frame part straight, left, FS 5		59.69.16.00.01.69
1.1	1	59690102	frame part abducted, right, FS 1		59.69.01.00.02.69
o.B.	1	59690101	frame part abducted, left, FS 1		59.69.01.00.01.69
1.1	1	59690202	frame part abducted, right, FS 2		59.69.02.00.02.69
o.B.	1	59690201	frame part abducted, left, FS 2		59.69.02.00.01.69
1.1	1	59690302	frame part abducted, right, FS3		59.69.03.00.02.69
o.B.	1	59690301	frame part abducted, left, FS 3		59.69.03.00.01.69
1.1	1	59690402	frame part abducted, right, FS 4		59.69.04.00.02.69
o.B.	1	59690401	frame part abducted, left, FS 4		59.69.04.00.01.69
1.1	1	59691102	frame part abducted, right, FS 5		59.69.15.00.02.69
o.B.	1	59691101	frame part abducted, left, FS 5		59.69.15.00.01.69
1.2	1	59690902	frame part adducted, right FS 3		59.69.09.00.02.69
o.B.	1	59690901	frame part adducted, left FS 3		59.69.09.00.01.69
1.2	1	59691002	frame part adducted, right FS 4		59.69.10.00.02.69
o.B.	1	59691001	frame part adducted, left FS 4		59.69.10.00.01.69
2	2	5834SB	crossbar D=24x2xSW		58.34.01.00.00.69
3-5	2	583499	mounting set crossbar on frame		XXXXXXXXXXXX
3	2				01.42.08.59.62.60
4	2		saddle washer Ø12/6x2,5		35.04.16.01.00.10
5	2		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20
6-8	1	59752501	braking rail short incl. mounting set		59.75.25.01.00.00




Spare parts list Vector

Frame


o.B.	1	59752502	braking rail long incl. mounting set	59.75.25.02.00.00
o.B.	1	35007602		35.00.76.01.02.00
o.B.	1	35007601		35.00.76.01.01.00
7	2		lens head screw with hexagon socket ISO 7380 - M6x16	01.42.08.52.65.00
8	2		hexagon nut self-locking low form DIN 985 - M6	02.32.00.60.01.20

Frame XL

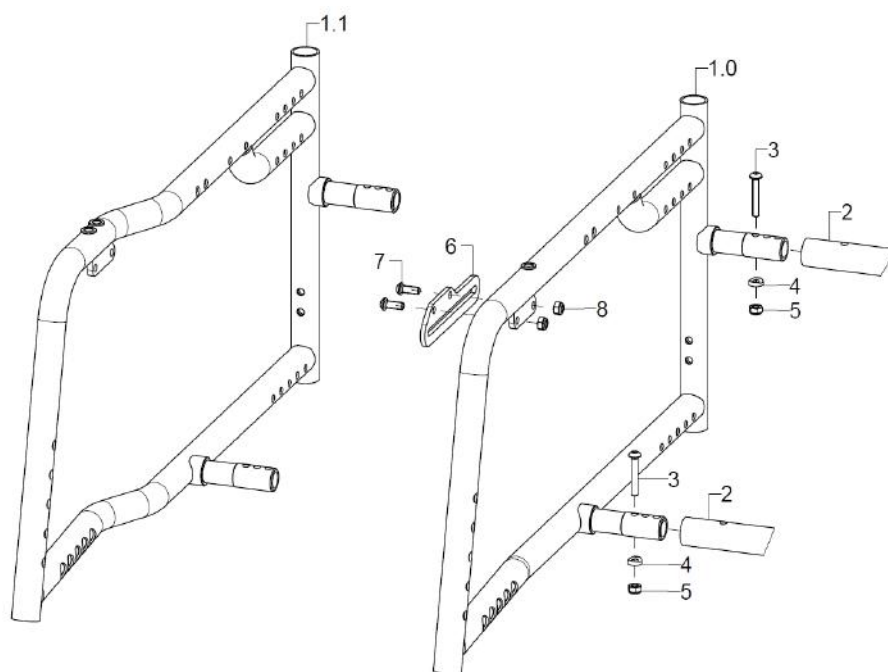

pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1.0	1	59691902			59.69.9.00.02.69
o.B.	1	59691901			59.69.19.00.01.69
1.1	1	59691802			59.69.18.00.02.69
o.B.	1	59691801			59.69.18.00.01.69
2	2	5834SB	crossbar D=24x2xSW		58.34.01.00.00.69
3-5	2	583499	mounting set crossbar on frame		XXXXXXXXXXXX
3	2				01.42.08.59.62.60
4	2		saddle washer Ø12/6x2,5		35.04.16.01.00.10
5	2		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20
6-8	1	59752502	braking rail long incl. mounting set		59.75.25.02.00.00
o.B.	1	35007602			35.00.76.01.02.00
o.B.	1	35007601			35.00.76.01.01.00
7	2		lens head screw with hexagon socket ISO 7380 - M6x16		01.42.08.52.65.00
8	2		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20

Frame extension for seat shell

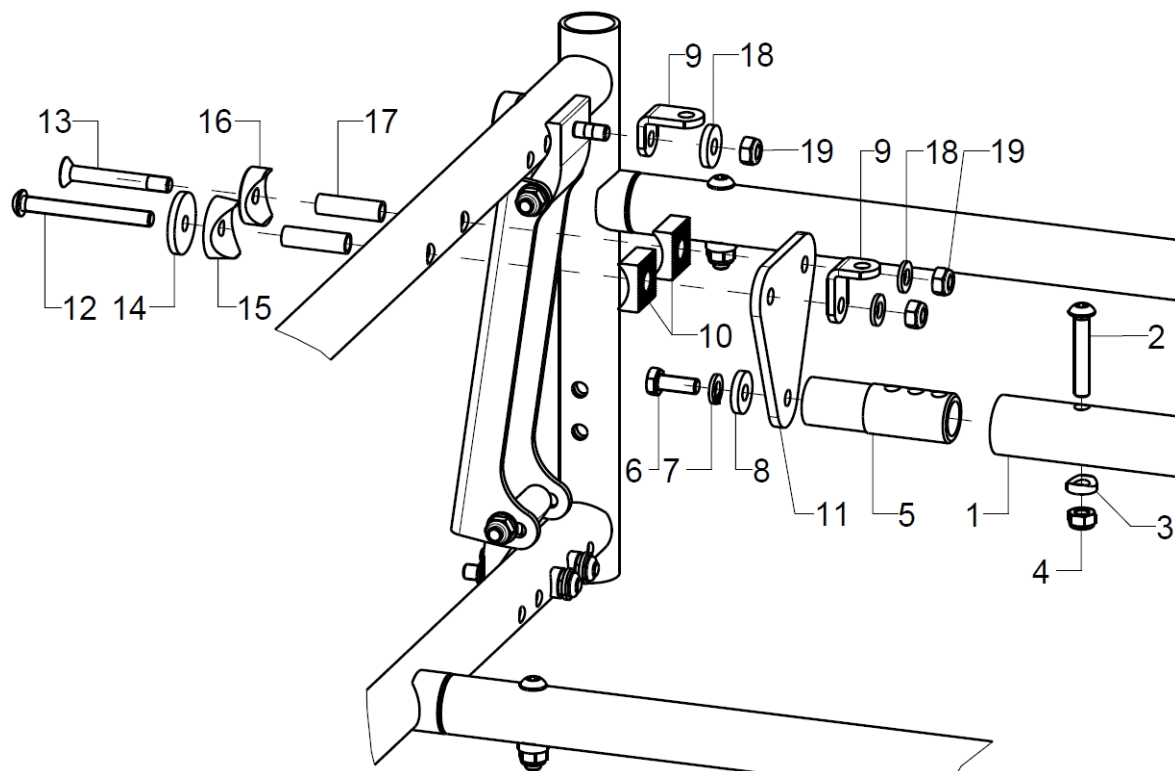

pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-19	1	58340100	frame extention for seat shells complete		XXXXXXXXXXXX
1	1	5834SB	crossbar D=24x2xSW		58.34.01.00.00.69
2-4	2	583499	mounting set crossbar on frame		XXXXXXXXXXXX
2	1				01.42.08.59.62.60
3	1				40.04.16.01.00.10
4	1		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20
5	2	583402	crossbar adapter, RD 20x64		58.34.01.06.00.69
6-8	2	58340299	mounting set traverse adapter		XXXXXXXXXXXX
6	1				01.03.08.52.02.40
7	1		spring ring DIN 127- A6		02.85.30.60.05.10
8	1				02.77.06.22.00.20
9	4	402705	joint angle L 25x19x3x15		40.27.05.01.00.10
10	4	190835	half shell 20 R12, D8		19.08.01.02.00.69
11	2	583403	crossbar adapter BI 4x65x74,5, black anodized		19.13.02.01.00.69
12-19	2	58340399	mounting set traverse holder on frame		XXXXXXXXXXXX
12	1		lens head screw with hexagon socket ISO 7380 - M6x55		01.42.08.63.65.00
13	1		counter sunk screw with hexagon socket DIN 7991 M6X45		01.65.08.61.05.00
14	1		washer D 25/d=6,2x3		28.00.21.01.00.10
15	1		saddle washer black ø 24/6		04.63.39.24.06.30
16	1				20.15.09.05.00.69
17	2				27.00.05.05.00.10
18	3		washer DIN 125 - A 6,4		02.61.06.41.41.20
19	3		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20

This exploded perspective view illustrates the assembly of a mechanical component. The main assembly includes a base plate (1) with a vertical section (11) and a horizontal section (19). A central shaft (12) is shown with a pin (13) at the top and a nut (14) at the bottom. A bracket (3) is attached to the vertical section (11) via a pin (15) and a nut (16). A handle (20) is attached to the horizontal section (19) via a pin (21) and a nut (22). A pin (24) is shown inserted into the handle (20). A pin (25) is shown inserted into the horizontal section (19). A pin (17) is shown inserted into the vertical section (11). A pin (18) is shown inserted into the horizontal section (19). A pin (4) is shown inserted into the vertical section (11). A pin (5) is shown inserted into the vertical section (11). A pin (6) is shown inserted into the bracket (3). A pin (7) is shown inserted into the bracket (3). A pin (8) is shown inserted into the horizontal section (19). A pin (9) is shown inserted into the horizontal section (19). A pin (10) is shown inserted into the horizontal section (19). A pin (11) is shown inserted into the horizontal section (19). A pin (12) is shown inserted into the horizontal section (19). A pin (13) is shown inserted into the horizontal section (19). A pin (14) is shown inserted into the horizontal section (19). A pin (15) is shown inserted into the horizontal section (19). A pin (16) is shown inserted into the horizontal section (19). A pin (17) is shown inserted into the horizontal section (19). A pin (18) is shown inserted into the horizontal section (19). A pin (19) is shown inserted into the horizontal section (19). A pin (20) is shown inserted into the horizontal section (19). A pin (21) is shown inserted into the horizontal section (19). A pin (22) is shown inserted into the horizontal section (19). A pin (23) is shown inserted into the horizontal section (19). A pin (24) is shown inserted into the horizontal section (19). A pin (25) is shown inserted into the horizontal section (19).




Spare parts list Vector

Frame

Page 6 of 127


pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1+4-7+11-25	1	721501/02	anti-tipper complete		XXXXXXXXXXXXX
1+8-25	1	721505/06	anti-tipper for wheel base extension complete		XXXXXXXXXXXXX
1	1	72150106	anti-tipper holder		72.15.01.06.00.74
2-7+14	1	721562	anti-tipper adapter SA		72.15.62.01.00.69
3	1				72.15.62.02.00.69
4-7	1	7215010699	mounting set anti-tipper holder		XXXXXXXXXXXXX
4	2		cylinder screw DIN 912 M6x35		01.22.08.59.62.40
altern.	2				01.22.08.66.02.40
5	2				02.98.06.05.00.20
6	2		fitted washer DIN 988 - 6x12x0,5 - A2		02.68.06.00.50.30
altern.	2		saddle washer Ø12x3x7		40.13.01.03.00.24
7	2		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20
8-10	1	7215020699	mounting set anti-tipper holder wheel base extension		
8	2				01.42.08.59.62.60
9	4		washer DIN 125 - A 6,4		02.61.06.41.41.20
10	2		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20
11-25	1	721500XX	anti-tipper pre-assembled		72.15.XX.99.XX.74
11	1		anti-tipper bail		72.15.01.XX.00.74
12	1		tension spring, loop 180°, 1/1 centred, 13,5x2x113		04.22.01.41.13.00
13	1		spacer bushing 18,4 mm		45.03.10.00.00.73
14	1		plugs for tube Ø22x1-2, black		04.63.10.22.00.10
15	1		lens head screw with hexagon socket ISO 7380 - M6x30		01.42.08.58.65.00
16	1		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20
17	1		cylinder screw DIN 912 M4x30		01.22.07.38.62.40
18	1		hexagon nut self-locking low form DIN 985 - M4		02.32.00.40.01.20
19	1	72150104	tipping lever black		72.15.01.04.00.74
20	1	720004	wheel Ø50, width 18, nob width 20, axle Ø 6, PU grey		04.52.42.50.00.30
21	1	72150101	holder anti-tipper for wheel Ø50 mm, incl. mounting set		72.15.01.01.00.74
22	1		cylinder screw DIN 912 M4x30		01.22.07.38.62.40
23	1		hexagon nut self-locking low form DIN 985 - M4		02.32.00.40.01.20
24	1		lens head screw with hexagon socket ISO 7380 - M6x40		01.42.08.60.65.00
25	1		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20

Tipping aid

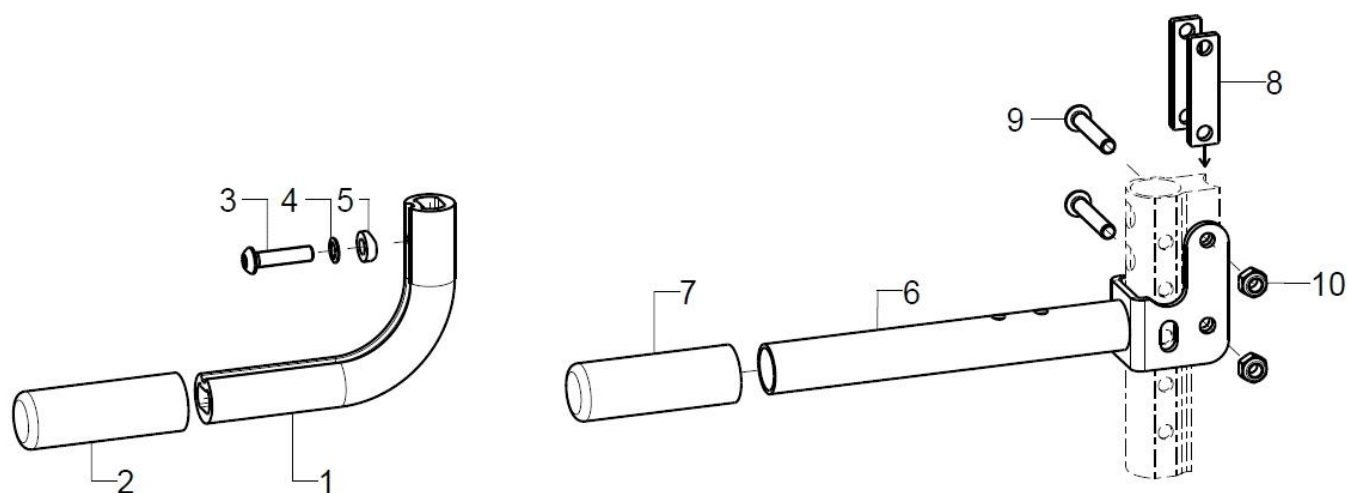


pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-5	1	7401	tipping lever complete		
1	1				74.01.01.01.00.10
2	1	740102	covered cap		74.01.02.01.00.10
3-5	1	74000099	mounting set for tipping lever		
3	1		lens head screw with hexagon socket ISO 7380 - M6x25		01.42.08.56.65.00
4	1		lock washer S6		02.99.10.60.01.10
5	1				40.04.16.01.00.10
6-10	1	7404			74.04.02.00.00.74
7	1	740102	covered cap		74.01.02.01.00.10
8	2				74.04.02.03.00.74
9	2		lens head screw with hexagon socket ISO 7380 - M6x35		01.42.08.59.65.00
10	2		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20

Crutch holder with tipping aid

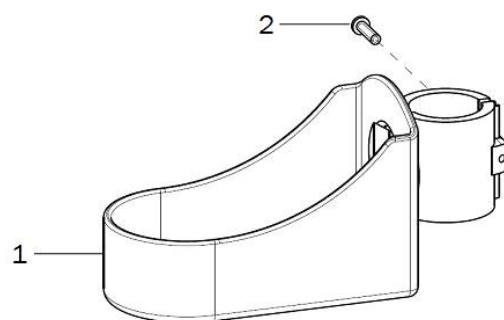


pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1	1	7101	crutch holder with mounting screw		71.01.01.00.00.10
2	1				

Leg support standard, internal mounting 403901

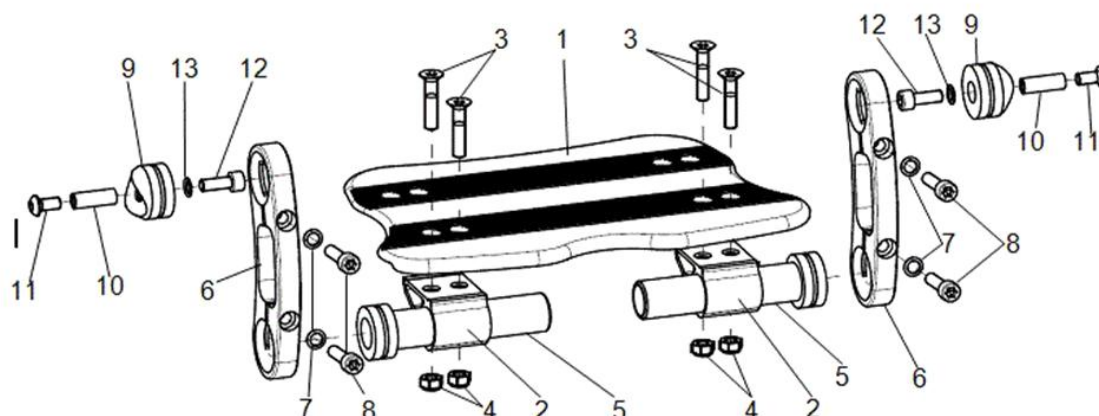


pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-13	1	403901	leg support continuous standard, inner attachment, complete		
1	1	40391500	foot rest for continuous leg support standard		40.39.16.00.00.69
2	2	40390106	clamp profile 46,5x28x30, black anodized		40.39.01.06.00.69
3+4	1	40390699	mounting set for clamp profile		
3	4		counter sunk screw with hexagon socket DIN 7991 M6x30		01.65.08.58.02.40
4	4		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20
5	2	40390102	adapter piece 96 mm (inner attachment)		40.39.01.02.00.69
6	2	40390101	positioning part 12x125x41,9		40.39.01.01.00.69
altern.	2	40390110	positioning part 12x175x42,1		40.39.01.10.00.69
7+8	2	40390199	mounting set positioning part		
7	2		lock washer S6		02.99.10.60.01.10
8	2		cylinder screw DIN 912 M6x25		01.22.08.56.02.40
9	2	40390103	foot plate attachment		40.39.01.03.00.69
10-13	2	40390399	mounting set for foot plate attachment		
10	1				59.95.02.13.00.10
11	1		lens head screw with hexagon socket ISO 7380 - M6x12		01.42.08.49.65.00
12	1		cylinder screw DIN 912 M6x16		01.22.08.52.62.40
13	1		lock washer S6		02.99.10.60.01.10

Leg support standard, external mounting 404001

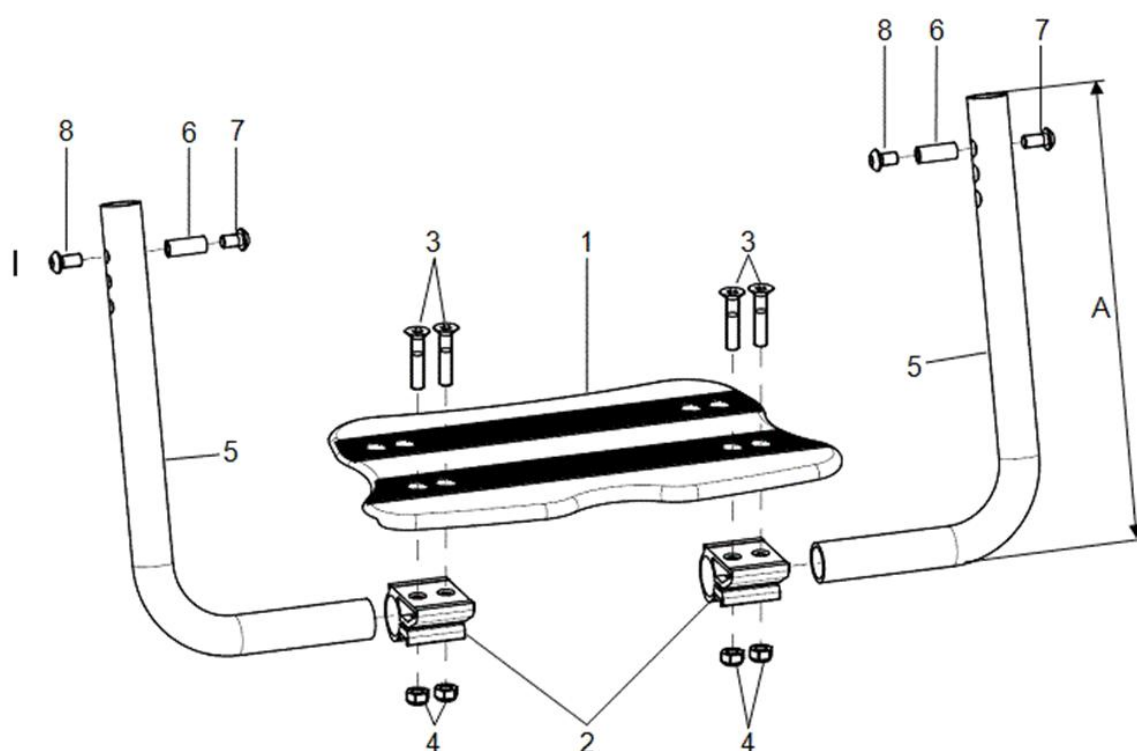

pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-8	1	404001	leg support continuous standard, outer attachment complete		
1	1	40391500	foot rest for continuous leg support standard		40.39.16.00.00.69
2	2	40390106	clamp profile 46,5x28x30, black anodized		40.39.01.06.00.69
3+4	2	40390699	mounting set for clamp profile		
3	2		counter sunk screw with hexagon socket DIN 7991 M6x30		01.65.08.58.02.40
4	2		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20
5	2	40390201	leg support tube standard (230mm)		40.39.02.01.00.69
altern.	2	40390202	leg support tube standard (310mm)		40.39.02.02.00.69
6-8	2	40390299	mounting set leg support tube		40.39.02.99.90.00
6	1		bushing M6x19,5		40.39.01.04.50.69
7	1		lens head screw with hexagon socket ISO 7380 - M6x10		01.42.08.48.65.00
8	1		lens head screw with hexagon socket ISO 7380 - M6x12		01.42.08.49.65.00

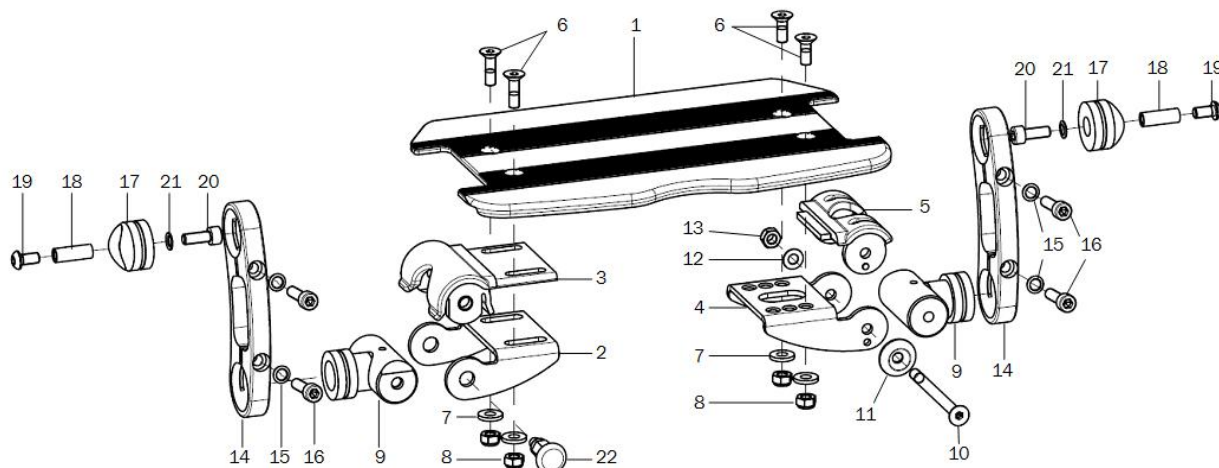
Leg support undivided, hinged sidewards, internal mounting, (version from July 2013)


pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-21	1	403601	leg support continuous standard, inner attachment complete		
1	1	40360710			40.36.07.00.00.74
altern.	1	40280100	continuous foot plate 8 mm		40.28.01.01.00.10
2-3+9+14-21		40362001	inner right attachment complete		40.36.20.01.90.00
4-5+9-21		40362101	inner left attachment of leg support, joint side complete		40.36.21.01.90.00
2-3	1	40360106	adjustment part right incl. raster piece		40.36.01.06.00.74
3	1	40360101	locking piece		40.36.01.01.00.74
4-5	1	40360501	adjustment part left for inner attachment incl. raster		40.36.05.01.01.74
5	1	40360102	raster part for foot plate		40.36.01.02.00.74
6-8	2	40360599	mounting set adjustment piece		XXXXXXXXXXXX
6	2		counter sunk screw with hexagon socket DIN 7991 M6x20		01.65.08.54.02.40
7	2		washer reinforced Ø14/6,2x2		58.30.15.01.00.24
8	2		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20
9	2	40360204	adapter for inner attachment incl. mounting set		40.36.02.04.00.74
10-13	1	40360299	mounting set adjustment piece		XXXXXXXXXXXX
10	1		counter sunk screw with hexagon socket DIN 7991 M6x75		01.65.08.67.02.40
11	1		rosette ring Ø22/6x4,5, black		04.14.70.60.09.00
12	1		fitted washer DIN 988 - 6x12x0,5 - A2		02.68.06.00.50.30
13	1		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20
14	2	40390101	positioning part 12x125x41,9		40.39.01.01.00.69
altern.	2	40390110	positioning part 12x175x42,1		40.39.01.10.00.69
15-16	2	40390199	mounting set positioning part		
15	2		lock washer S6		02.99.10.60.01.10
16	2		cylinder screw DIN 912 M6x25		01.22.08.56.02.40
17	2	40390103	foot plate attachment		40.39.01.03.00.69
18-21	2	40390399	mounting set for foot plate attachment		
18	1				59.95.02.13.00.10
19	1		lens head screw with hexagon socket ISO 7380 - M6x12		01.42.08.49.65.00
20	1		cylinder screw DIN 912 M6x16		01.22.08.52.62.40
21	1		lock washer S6		02.99.10.60.01.10




22	1	690101	latch ø6/M10x1x31 with lock	04.14.43.60.02.00
2+9+22	1	400902	lock complete	

Leg support undivided, hinged sideways, internal mounting, (version until June 2013)

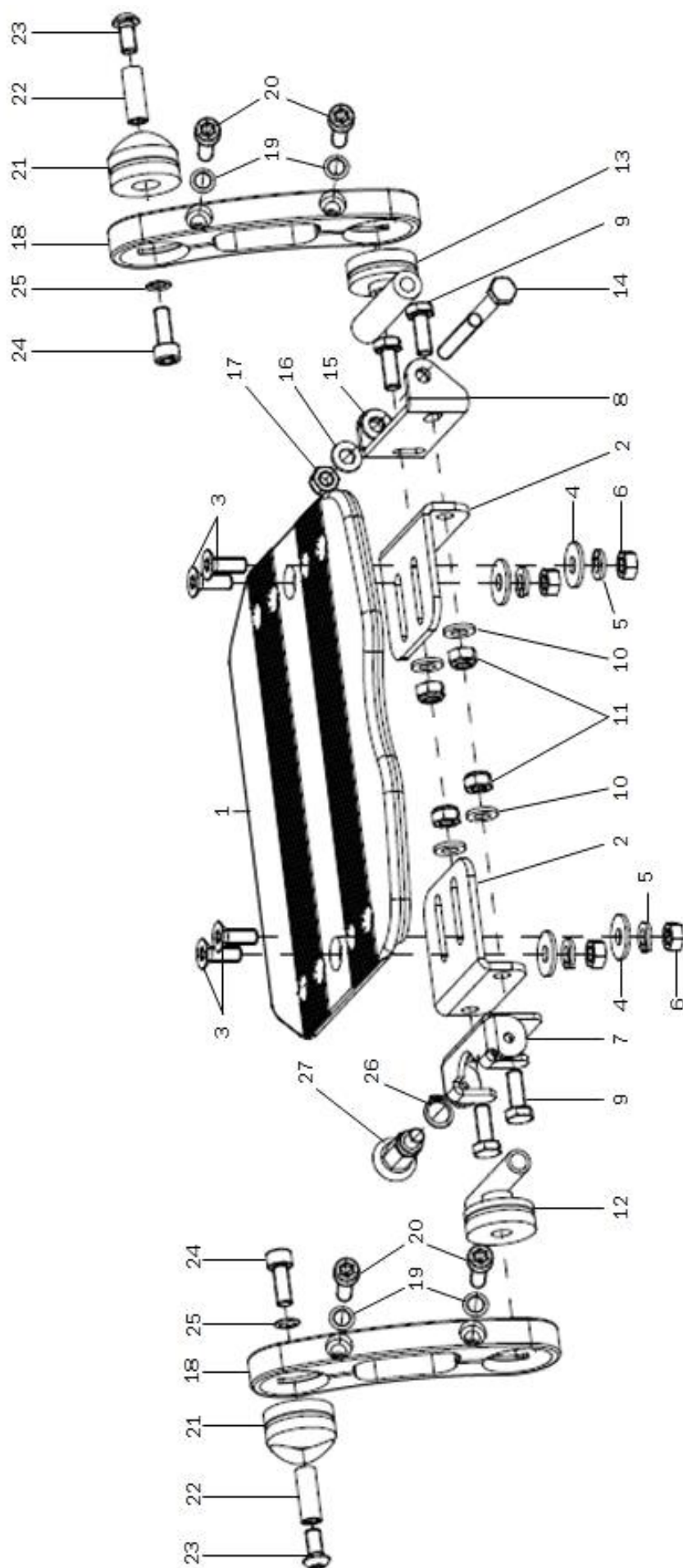




pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-25	1	403902	leg support continuous, folds up sideways, inner attachment complete		
1	1	40280100	continuous foot plate 8 mm		40.28.01.01.00.10
2	2	402820	angle for foot plate continuous		40.28.01.02.00.10
3-6	2	40282099	mounting set angle part		
3	2		counter sunk screw with hexagon socket DIN 7991 M6x16		01.65.08.52.02.40
4	2		washer DIN 9021 -6,4		02.79.06.41.60.20
5	2		spring ring DIN 127- A6		02.85.30.60.05.10
6	2		hexagon nut DIN 934 - M10		02.19.01.00.01.20
7	1	400306	foot plate raster lockable		40.03.06.08.00.10
8	1	400004	foot plate joint continuous angle adjustable		40.22.04.01.01.20
9-11	2	40000699	mounting set foot plate raster and joint		
9	2				01.03.08.52.02.40
10	2		washer DIN 125 - A 6,4		02.61.06.41.41.20
11	2		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20
12	1	40390108	adapter piece right		40.39.01.07.02.69
13	1	40390107	adapter piece left		40.39.01.07.01.69
14-17	1	40001799	mounting set adapter piece left		
14	1				
15	2				
16	1		washer DIN 125 - A 6,4		02.61.06.41.41.20
17	1		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20
18	2	40390101	positioning part 12x125x41,9		40.39.01.01.00.69
altern.	2	40390110	positioning part 12x175x42,1		40.39.01.10.00.69
19-20	2	40390199	mounting set positioning part		
19	2		lock washer S6		02.99.10.60.01.10
20	2		cylinder screw DIN 912 M6x25		01.22.08.56.02.40
21	2	40390103	foot plate attachment		40.39.01.03.00.69
22-25	2	40390399	mounting set for foot plate attachment		
22	1				59.95.02.13.00.10
23	1		lens head screw with hexagon socket ISO 7380 - M6x12		01.42.08.49.65.00
24	1		cylinder screw DIN 912 M6x16		01.22.08.52.62.40
25	1		lock washer S6		02.99.10.60.01.10
26	1		lock ring 10, DIN 471		03.01.01.00.93.00
27	1	690101	latch ø6/M10x1x31 with lock		04.14.43.60.02.00

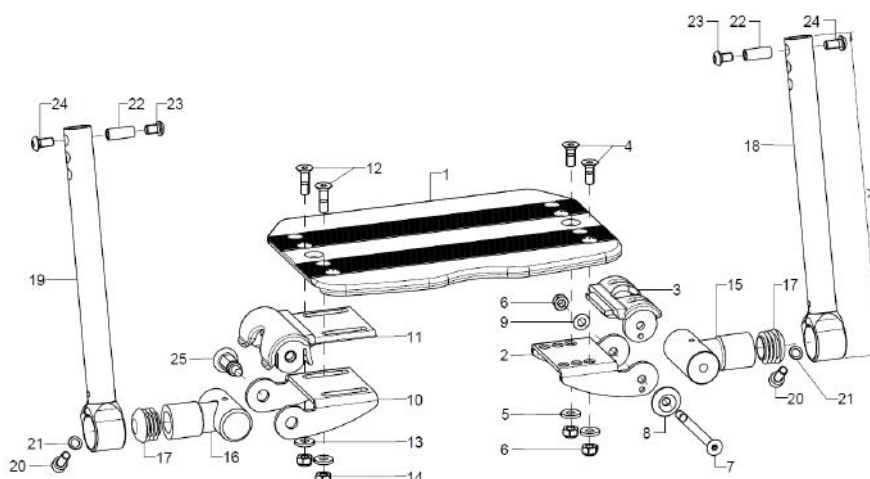Leg support undivided, hinged sideways, external mounting, (version from July 2013)


pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-24	1	404002	leg support continuous, folds up sideways, outer attachment complete		
1	1	40280100	continuous foot plate 8 mm		40.28.01.01.00.10
2-3	1	40360105	adjustment part left incl. raster piece		40.36.01.05.00.74
3	1	40360102	raster part for foot plate		40.36.01.02.00.74
4-9	1	40360299	mounting set adjustment piece		XXXXXXXXXXXX
4	2		counter sunk screw with hexagon socket DIN 7991 M6x16		01.65.08.52.02.40
5	2		washer reinforced Ø14/6,2x2		58.30.15.01.00.24
6	3		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20
7	1		counter sunk screw with hexagon socket DIN 7991 M6x75		01.65.08.67.02.40
8	1		rosette ring Ø22/6x4,5, black		04.14.70.60.09.00
9	1		fitted washer DIN 988 - 6x12x0,5 - A2		02.68.06.00.50.30
10-11	1	40360106	adjustment part right incl. raster piece		40.36.01.06.00.74
11	1	40360101	locking piece		40.36.01.01.00.74
12-14		40360599	mounting set adjustment piece		XXXXXXXXXXXX
12	2		counter sunk screw with hexagon socket DIN 7991 M6x20		01.65.08.54.02.40
13	2		washer reinforced Ø14/6,2x2		58.30.15.01.00.24
14	2		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20
15+17	1	40360111	joint D6 incl. plug		40.36.01.11.50.74
16-17	1	40360110	joint incl. plug		40.36.01.10.50.74
o.B.	1	40360112	joint for latch incl. plug		40.36.01.12.50.74
17	2		plugs for tube Ø25x1-2, black		04.63.10.25.00.10
18+20+21	1	40360121	leg support tube glued 235 mm left, incl. mounting screws		40.36.01.15.21.74
o.B.	1	40360131	leg support tube glued 305 mm left, incl. mounting screws		40.36.01.15.31.74
19-20+21	1	40360122	leg support tube glued 235 mm right, incl. mounting screws		40.36.01.15.22.74
o.B.	1	40360132	leg support tube glued 305 mm right, incl mounting screws		40.36.01.15.32.74
20	1		cylinder screw DIN 912 M6x16		01.22.08.52.62.40
21	1		lock washer S6		02.99.10.60.01.10
22-24	2	40390299	mounting set leg support tube		40.39.02.99.90.00
22	1		bushing M6x19,5		40.39.01.04.50.69
23	1		lens head screw with hexagon socket ISO 7380 - M6x10		01.42.08.48.65.00
24	1		lens head screw with hexagon socket ISO 7380 - M6x12		01.42.08.49.65.00
25	1	690101	latch Ø6/M10x1x31 with lock		04.14.43.60.02.00




Leg support undivided, hinged sideways, external mounting, (version until June 2013)

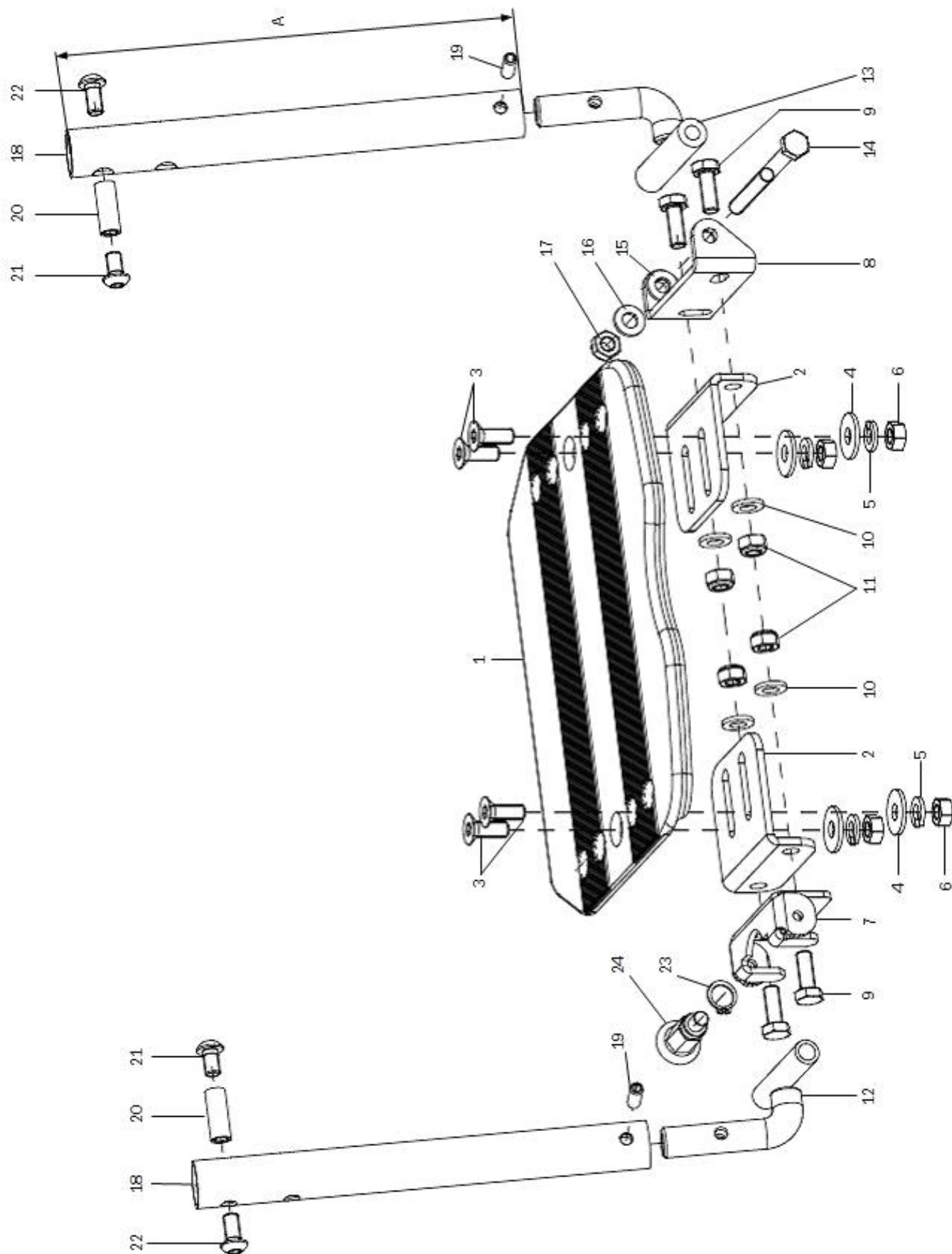


pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-22	1	404002	leg support continuous, folds up sideways, outer attachment complete		
1	1	40280100	continuous foot plate 8 mm		40.28.01.01.00.10
2	2	402820	angle for foot plate continuous		40.28.01.02.00.10
3-6	2	40282099	mounting set angle part		
3	2		counter sunk screw with hexagon socket DIN 7991 M6x16		01.65.08.52.02.40
4	2		washer DIN 9021 -6,4		02.79.06.41.60.20
5	2		spring ring DIN 127- A6		02.85.30.60.05.10
6	2		hexagon nut DIN 934 - M10		02.19.01.00.01.20
7	1	400306	foot plate raster lockable		40.03.06.08.00.10
8	1	400004	foot plate joint continuous angle adjustable		40.22.04.01.01.20
9-11	2	40000699	mounting set foot plate raster and joint		
9	2				01.03.08.52.02.40
10	2		washer DIN 125 - A 6,4		02.61.06.41.41.20
11	2		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20
12	1	400018	adapter 35 (75°) right		40.00.18.01.00.10
13	1	400017	adapter 35 (75°) left		40.00.17.01.00.10
14-17	1	40001799	mounting set adapter piece left		
14	1				
15	2				
16	1		washer DIN 125 - A 6,4		02.61.06.41.41.20
17	1		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20
18	2	40400101	leg support tube 180mm		40.40.01.03.00.69
altern.	2	40400102	leg support tube 240mm		40.40.01.04.00.69
19-22	2	40400199	mounting set leg support tube		
19	1				
20	1		bushing M6x20,5		40.39.01.04.00.69
21	1		lens head screw with hexagon socket ISO 7380 - M6x10		01.42.08.48.65.00
22	1		lens head screw with hexagon socket ISO 7380 - M6x12		01.42.08.49.65.00
23	1		lock ring 10, DIN 471		03.01.01.00.93.00
24	1	690101	latch ø6/M10x1x31 with lock		04.14.43.60.02.00

Achtung: Wenn bisher eine andere, als die oben abgebildete Fußplatte verwendet wurde, bitte bei Bestellung der Fußplatte auch Pos. 2 und Montagezubehör für Winkelteile mitbestellen.
Maß X der Adapter Pos. 11 + 12 angeben.

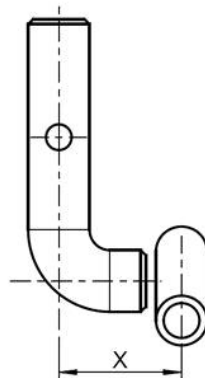

Leg support undivided, hinged backwards, internal mounting, N° 403903

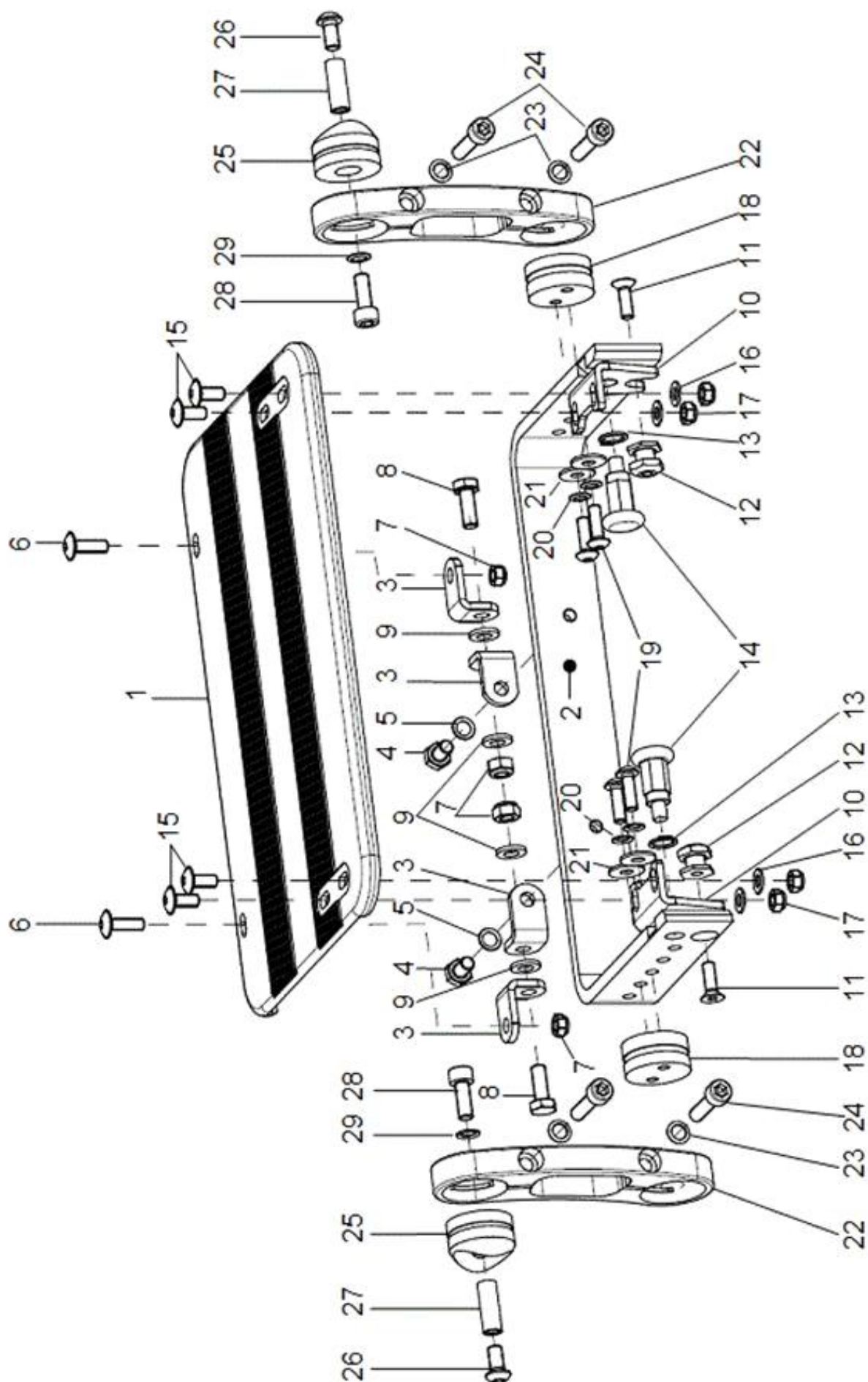




pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-29	1	403903	leg support continuous, folds up to the back, inner attachment complete		
1	1	40480100	continuous foot plate for leg support, folds up to the back, inner attachment		40.48.01.XX.XX.10
2	1	40480300	folding bail		40.40.05.01.00.69
3-9	1	40480399	mounting set for inner folding bail		
3	4				
4	2				01.04.08.48.02.40
5	2		lock washer S6		02.99.10.60.01.10
6	2		lens head screw with hexagon socket M5x20		01.43.07.92.05.00
o.B.	2		hexagon nut DIN 934 - M6		02.19.00.60.01.20
7	4		hexagon nut self-locking low form DIN 985 - M5		02.32.00.50.01.20
8	2		hexagon screw DIN 933 - M6x16		01.04.08.52.02.40
9	4				
10	2	402909	angle holder for latch L 31x20x3x34		40.29.04.01.00.10
11-17	2	40290999	mounting set angle holder		
11	1		counter sunk screw with hexagon socket DIN 7991 M5x16		01.65.07.92.05.00
12	1				
13	1		lock ring 10, DIN 471		03.01.01.00.93.00
14	1		latch ø6/M10x1x31 with lock		04.14.43.60.02.00
15	2		lens head screw with hexagon socket M5x12		01.43.07.89.65.00
16	2		washer DIN 125 - A 5,3		02.61.05.31.01.20
17	2		hexagon nut self-locking low form DIN 985 - M5		02.32.00.50.01.20
18	2	40390301	attachment for foldable foot plate		40.39.03.01.00.69
19-21	2	40390399	mounting set for foot plate attachment		
19	2		lens head screw with hexagon socket ISO 7380 - M5x16		01.42.07.92.65.00
20	2		lock washer S5		02.99.10.50.01.10
21	2		washer DIN 9021 - M5		02.79.05.31.20.20
22	2	40390101	positioning part 12x125x41,9		40.39.01.01.00.69
altern.	2	40390110	positioning part 12x175x42,1		40.39.01.10.00.69
23-24	2	40390199	mounting set positioning part		
23	2		lock washer S6		02.99.10.60.01.10
24	2		cylinder screw DIN 912 M6x25		01.22.08.56.02.40
25	2	40390103	foot plate attachment		40.39.01.03.00.69
26-29	2	40390399	mounting set for foot plate attachment		
26	1				59.95.02.13.00.10
27	1		lens head screw with hexagon socket ISO 7380 - M6x12		01.42.08.49.65.00
28	1		cylinder screw DIN 912 M6x16		01.22.08.52.62.40
29	1		lock washer S6		02.99.10.60.01.10

Achtung: Wenn bisher eine andere, als die oben abgebildete Fußplatte verwendet wurde, bitte bei Bestellung der Fußplatte auch Pos. 2, 18 und Montagezubehör für Anbindungen, Pos. 19 – 21 mitbestellen.


Leg support undivided, hinged backwards, external mounting, N° 403904 (version from September 2013)

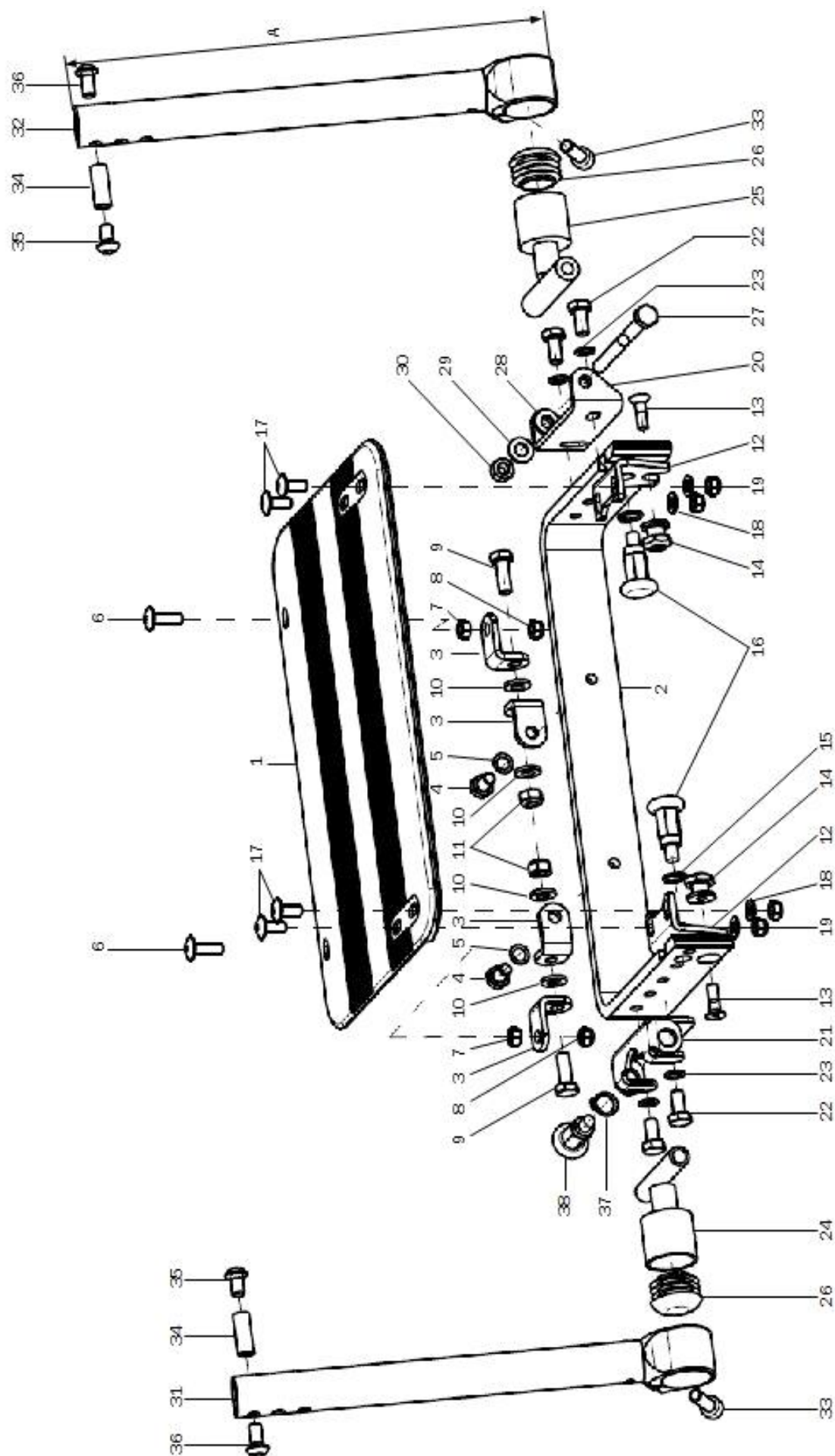



pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-36	1	403904	leg support continuous, folds up to the back, outer attachment complete		
1	1	40480100	continuous foot plate for leg support, folds up to the back, inner attachment		40.48.01.XX.XX.10
2	1	40480300	folding bail		40.40.05.01.00.69
3-11	1	40480399	mounting set for inner folding bail		
3	4				
4	2				01.04.08.48.02.40
5	2		lock washer S6		02.99.10.60.01.10
6	2		lens head screw with hexagon socket M5x20		01.43.07.92.05.00
7	2		hexagon nut DIN 934 - M6		02.19.00.60.01.20
8	2		hexagon nut self-locking low form DIN 985 - M5		02.32.00.50.01.20
9	2		hexagon screw DIN 933 - M6x16		01.04.08.52.02.40
10	4				
11	2		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20
12	2	402909	angle holder for latch L 31x20x3x34		40.29.04.01.00.10
13-19	2	40290999	mounting set angle holder		
13	1		counter sunk screw with hexagon socket DIN 7991 M5x16		01.65.07.92.05.00
14	1				
15	1		lock ring 10, DIN 471		03.01.01.00.93.00
16	1		latch ø6/M10x1x31 with lock		04.14.43.60.02.00
17	2		lens head screw with hexagon socket M5x12		01.43.07.89.65.00
18	2		washer DIN 125 - A 5,3		02.61.05.31.01.20
19	2		hexagon nut self-locking low form DIN 985 - M5		02.32.00.50.01.20
20	1	400004	foot plate joint continuous angle adjustable		40.22.04.01.01.20
21	1	400306	foot plate raster lockable		40.03.06.08.00.10
22-23	2	40030699	mounting set foot plate raster and joint		
22	2				01.03.08.52.02.40
23	2		lock washer S6		02.99.10.60.01.10
24+26	1				
25-26	1	40400503	adapter FR back left complete		40.40.05.03.00.69
26	1		plugs for tube ø25x1-2, black		04.63.10.25.00.10
27-30	1				
27	1				
28	2				
29	1		washer DIN 125 - A 6,4		02.61.06.41.41.20
30	1		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20
31+33	1	40360122	leg support tube glued 235 mm right, incl. mounting screws		40.36.01.15.22.74
o.B.	1	40360132	leg support tube glued 305 mm right, incl mounting screws		40.36.01.15.32.74
32-33	1	40360121	leg support tube glued 235 mm left, incl. mounting screws		40.36.01.15.21.74
o.B.	1	40360131	leg support tube glued 305 mm left, incl. mounting screws		40.36.01.15.31.74
33	1		cylinder screw DIN 912 M6x20		01.22.08.54.62.40
34-36	2	40400199	mounting set leg support tube		
34	1		bushing M6x19,5		40.39.01.04.50.69
35	1		lens head screw with hexagon socket ISO 7380 - M6x10		01.42.08.48.65.00
36	1		lens head screw with hexagon socket ISO 7380 - M6x12		01.42.08.49.65.00
37	1		lock ring 10, DIN 471		03.01.01.00.93.00
38	1		latch ø6/M10x1x31 with lock		04.14.43.60.02.00

Leg support undivided, hinged backwards, external mounting, N° 403904 (version until August 2013)

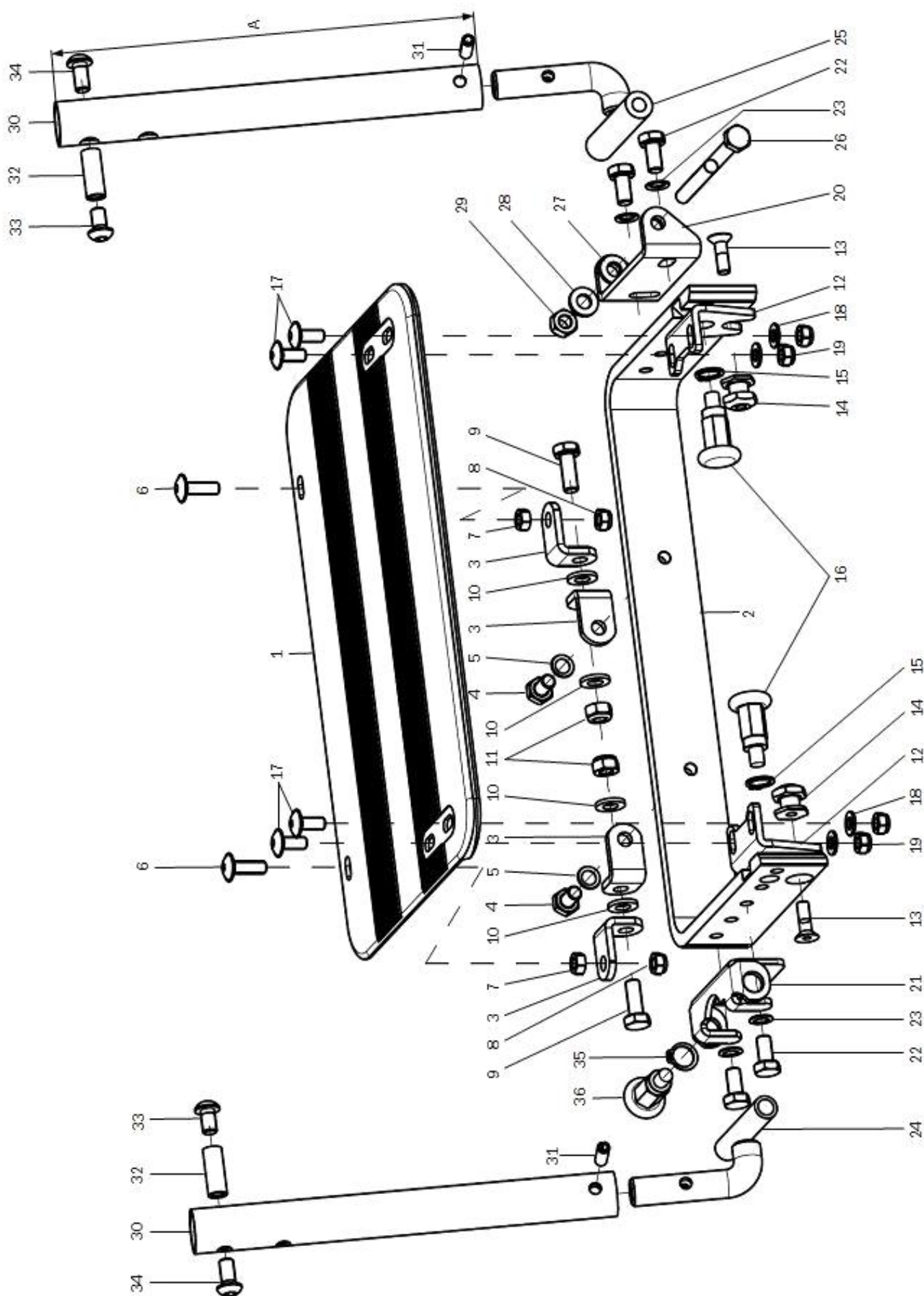



Spare parts list Vector


Leg support


Page 24 of 127

pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-34	1	403904	leg support continuous, folds up to the back, outer attachment complete		
1	1	40480100	continuous foot plate for leg support, folds up to the back, inner attachment		40.48.01.XX.XX.10
2	1	40480300	folding bail		40.40.05.01.00.69
3-11	1	40480399	mounting set for inner folding bail		
3	4				
4	2				01.04.08.48.02.40
5	2		lock washer S6		02.99.10.60.01.10
6	2		lens head screw with hexagon socket M5x20		01.43.07.92.05.00
7	2		hexagon nut DIN 934 - M6		02.19.00.60.01.20
8	4		hexagon nut self-locking low form DIN 985 - M5		02.32.00.50.01.20
9	2		hexagon screw DIN 933 - M6x16		01.04.08.52.02.40
10	4				
11	2		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20
12	2	402909	angle holder for latch L 31x20x3x34		40.29.04.01.00.10
13-19	2	40290999	mounting set angle holder		
13	1		counter sunk screw with hexagon socket DIN 7991 M5x16		01.65.07.92.05.00
14	1				
15	1		lock ring 10, DIN 471		03.01.01.00.93.00
16	1		latch ø6/M10x1x31 with lock		04.14.43.60.02.00
17	2		lens head screw with hexagon socket M5x12		01.43.07.89.65.00
18	2		washer DIN 125 - A 5,3		02.61.05.31.01.20
19	2		hexagon nut self-locking low form DIN 985 - M5		02.32.00.50.01.20
20	1	400004	foot plate joint continuous angle adjustable		40.22.04.01.01.20
21	1	400306	foot plate raster lockable		40.03.06.08.00.10
22-23	2	40030699	mounting set foot plate raster and joint		
22	1		hexagon screw DIN 933 - M6x16		01.04.08.52.02.40
23	2		lock washer S6		02.99.10.60.01.10
24	1	400018	adapter 35 (75°) right		40.00.18.01.00.10
25	1	400017	adapter 35 (75°) left		40.00.17.01.00.10
26-29	1				
26	1				
27	2				
28	1		washer DIN 125 - A 6,4		02.61.06.41.41.20
29	1		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20
30	2	40400101	leg support tube 180mm		40.40.01.03.00.69
altern.	2	40400102	leg support tube 240mm		40.40.01.04.00.69
31-34	2	40400199	mounting set leg support tube		
31	1	034100501810	dowel pin DIN 1481 ø5x18		03.41.00.50.18.10
32	1		bushing M6x20,5		40.39.01.04.00.69
33	1		lens head screw with hexagon socket ISO 7380 - M6x10		01.42.08.48.65.00
34	1		lens head screw with hexagon socket ISO 7380 - M6x12		01.42.08.49.65.00
35	1		lock ring 10, DIN 471		03.01.01.00.93.00
36	1		latch ø6/M10x1x31 with lock		04.14.43.60.02.00

Achtung:

Wenn bisher eine andere, als die oben abgebildete Fußplatte verwendet wurde, bitte bei Bestellung der Fußplatte auch Pos. 2 mitbestellen.
Maß X der Adapter Pos. 24 + 25 angeben.

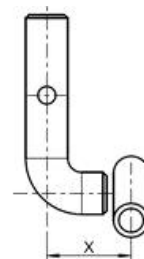

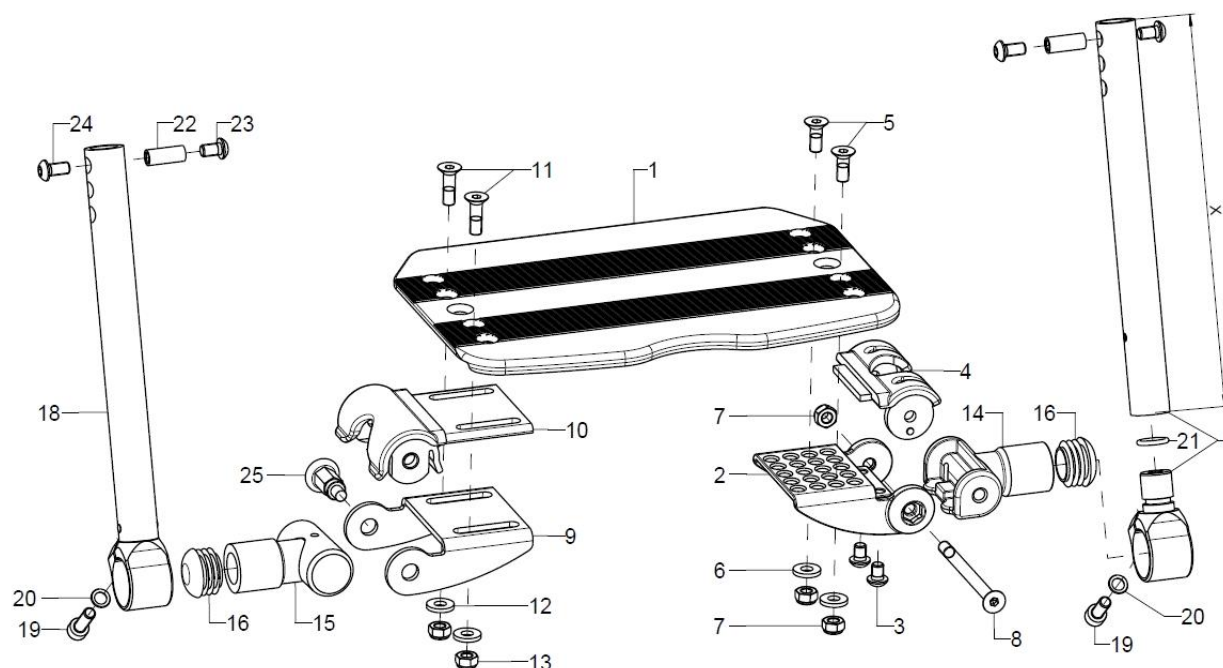Leg support undivided, hinged sideways and turnable outwards (version from May 2013)


pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-25	1	403611	leg support continuous, folds up sideways, swing-away, outer attachment complete		XXXXXXXXXXXX
1	1	40280100	continuous foot plate 8 mm		40.28.01.01.00.10
2-4	1	40361020	adjustment unit divided incl. grid		40.36.10.30.00.74
3	2		lens head screw with hexagon socket ISO 7380 - M6x8		01.42.08.47.65.00
4	1	40360102	raster part for foot plate		40.36.01.02.00.74
5-8	1	40360299	mounting set adjustment piece		XXXXXXXXXXXX
5	2		counter sunk screw with hexagon socket DIN 7991 M6x16		01.65.08.52.02.40
6	2		washer reinforced Ø14/6,2x2		58.30.15.01.00.24
7	3		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20
8	1		counter sunk screw with hexagon socket DIN 7991 M6x75		01.65.08.67.02.40
9-10	1	40360106	adjustment part right incl. raster piece		40.36.01.06.00.74
10	1	40360101	locking piece		40.36.01.01.00.74
11-13	2	40360599	mounting set adjustment piece		XXXXXXXXXXXX
11	2		counter sunk screw with hexagon socket DIN 7991 M6x20		01.65.08.54.02.40
12	2		washer reinforced Ø14/6,2x2		58.30.15.01.00.24
13	2		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20
14+16	1	40360205	joint for divided leg support incl. plug		40.36.10.31.00.74
15-16	1	40360110	joint incl. plug		40.36.01.10.50.74
altern.	1	40360112	joint for latch incl. plug		40.36.01.12.50.74
16	2		plugs for tube Ø25x1-2, black		04.63.10.25.00.10
17+19-21	1	40360320	leg support tube swingaway 235 mm, complete		40.36.03.04.20.74
o.B.	1	40360330	leg support tube detachable 305 mm		40.36.03.04.30.74
18-20	1	40360122	leg support tube glued 235 mm right, incl. mounting screws		40.36.01.15.22.74
o.B.	1	40360132	leg support tube glued 305 mm right, incl mounting screws		40.36.01.15.32.74
19	1		cylinder screw DIN 912 M6x16		01.22.08.52.62.40
20	1		lock washer S6		02.99.10.60.01.10
21	1		O-ring 12x2,2 mm, black		04.62.01.12.22.00
22-24	2	40390299	mounting set leg support tube		40.39.02.99.90.00
22	1		bushing M6x19,5		40.39.01.04.50.69




Spare parts list Vector

Leg support


23	1		lens head screw with hexagon socket ISO 7380 - M6x10	01.42.08.48.65.00
24	1		lens head screw with hexagon socket ISO 7380 - M6x12	01.42.08.49.65.00
25	1	690101	latch ø6/M10x1x31 with lock	04.14.43.60.02.00
9+15+2 5	1	400902	lock complete	

Leg support undivided, hinged sideways and turnable outwards (version until April 2013)

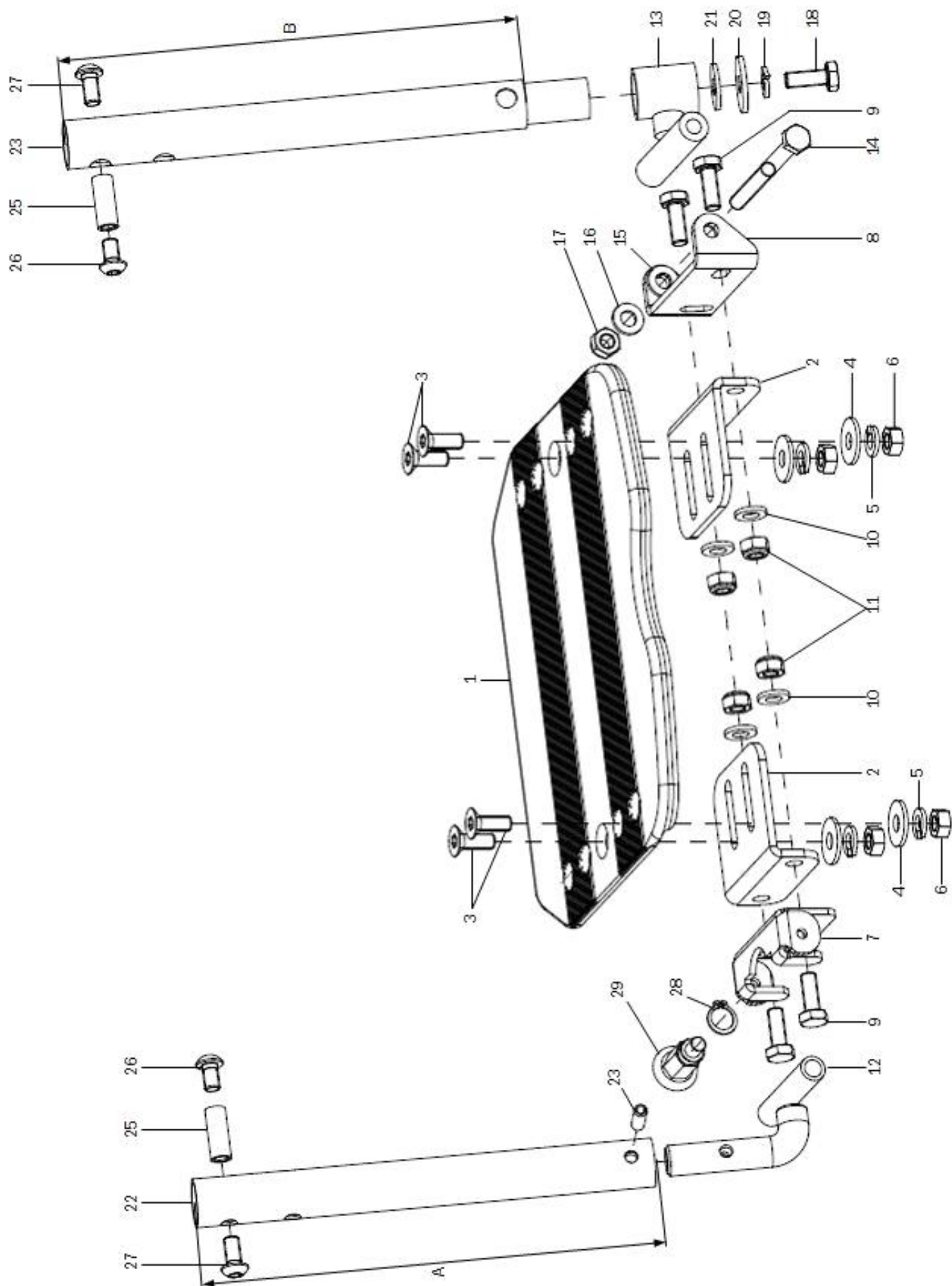


pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-27	1	404004	leg support continuous, folds up sideways, swing-away, outer attachment complete		
1	1	40280100	continuous foot plate 8 mm		40.28.01.01.00.10
2	2	402820	angle for foot plate continuous		40.28.01.02.00.10
3-6	2	40282099	mounting set angle part		
3	2		counter sunk screw with hexagon socket DIN 7991 M6x16		01.65.08.52.02.40
4	2		washer DIN 9021 -6,4		02.79.06.41.60.20
5	2		spring ring DIN 127- A6		02.85.30.60.05.10
6	2		hexagon nut DIN 934 - M10		02.19.01.00.01.20
7	1	400306	foot plate raster lockable		40.03.06.08.00.10
8	1	400004	foot plate joint continuous angle adjustable		40.22.04.01.01.20
9-11	2	40000699	mounting set foot plate raster and joint		
9	2				01.03.08.52.02.40
10	2		washer DIN 125 - A 6,4		02.61.06.41.41.20
11	2		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20
12	1	400018	adapter 35 (75°) right		40.00.18.01.00.10
13	1	402605	swing-away adapter 35		40.26.05.01.00.10
14-21	1	40260599	mounting set swing adapter		
14	1				
15	2				
16	1		washer DIN 125 - A 6,4		02.61.06.41.41.20
17	1		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20
18	1		hexagon screw DIN 933 - M6x16		01.04.08.52.02.40
19	1		spring ring DIN 127- A6		02.85.30.60.05.10
20	1		washer DIN 440, 6,6x22x2		02.67.22.03.09.20
21	1		washer DIN 9021- 6,4- PA		02.79.06.41.60.70
22	1	40400101	leg support tube 180mm		40.40.01.03.00.69
altern.	1	40400102	leg support tube 240mm		40.40.01.04.00.69
23	1	40400401	leg support tube detachable, 185 mm		40.40.04.01.00.69
altern.	1	40400402	leg support tube detachable, 245 mm		40.40.04.02.00.69
24-27	2	40400199	mounting set leg support tube		
24	1				
25	1		bushing M6x20,5		40.39.01.04.00.69
26	1		lens head screw with hexagon socket ISO 7380 - M6x10		01.42.08.48.65.00
27	1		lens head screw with hexagon socket ISO 7380 - M6x12		01.42.08.49.65.00
28	1		lock ring 10, DIN 471		03.01.01.00.93.00
29	1	690101	latch ø6/M10x1x31 with lock		04.14.43.60.02.00

Achtung: Wenn bisher eine andere, als die oben abgebildete Fußplatte verwendet wurde, bitte bei Bestellung der Fußplatte auch Pos. 2 mitbestellen.
Maß X der Adapter Pos. 11 + 12 angeben

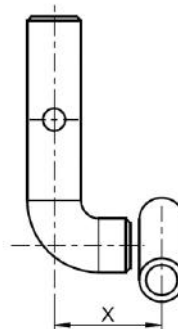


Foot rest undivided

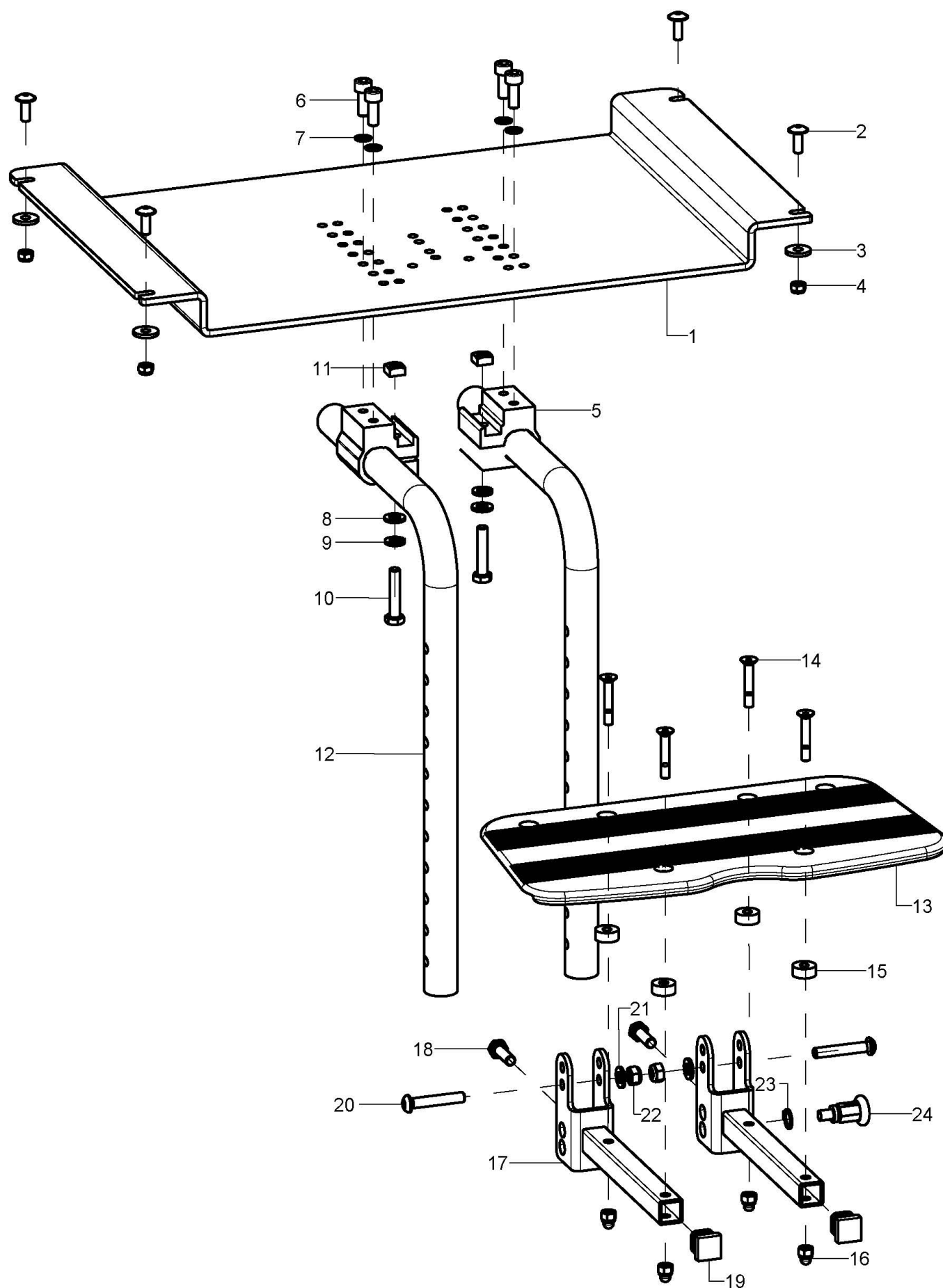




pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-22	1		leg support continuous, fold up to the back , double intake complete		
1-4	1	400830	solid leg support holder incl. mounting set		40.08.31.01.00.69
2	4		lens head screw with hexagon socket M5x14		01.43.07.90.62.60
3	4		washer DIN 7349 - 5,3		02.77.05.22.00.20
4	4		hexagon nut self-locking low form DIN 985 - M5		02.32.00.50.01.20
5-11	2	40551003			40.55.10.03.00.76
6	2		cylinder screw DIN 912 M6x16		01.22.08.52.62.40
7	2		lock washer S6		02.99.10.60.01.10
8	1		washer DIN 125 - A 6,4		02.61.06.41.41.20
9	1		spring ring DIN 127- A6		02.85.30.60.05.10
10	1		hexagon screw DIN 933 - M6x30		01.04.08.58.02.40
11	1		square nut DIN 557 M6		02.13.00.60.01.20
12	2	400102	knee angle leg support holder 90° 310mm		40.01.02.01.00.24
altern.	2	400111	knee angle leg support holder 90° 410 mm		40.01.02.01.00.28
altern.	2				
13-16	1	40551100	foot rest 150 mm continuous incl mounting set		40.55.11.00.00.76
14-16	1	40551199	mounting set foot rest		
14	4		counter sunk screw with hexagon socket DIN 7991 M5x30		01.65.07.98.05.00
15	4		bushing Ø14/5,2x6		40.55.00.01.00.76
16	4		hexagon nut self-locking low form DIN 985 - M5		02.32.00.50.01.20
17-22	2	400103	foot plate joint holder 100720 for continuous foot plate incl. mounting set		40.01.03.02.00.28
18	1		hexagon screw DIN 933 - M6x20		01.04.08.54.02.40
19	1		plugs for square tubes, black 15x15		04.63.11.15.00.10
20	1		lens head screw with hexagon socket ISO 7380 - M6x55		01.42.08.63.65.00
21	1		washer DIN 125 - A 6,4		02.61.06.41.41.20
22	1		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20
24	1	690101	latch ø6/M10x1x31 with lock		04.14.43.60.02.00
23	1		lock ring 10, DIN 471		03.01.01.00.93.00

Leg support divided, hinged sideways, internal mounting 403602

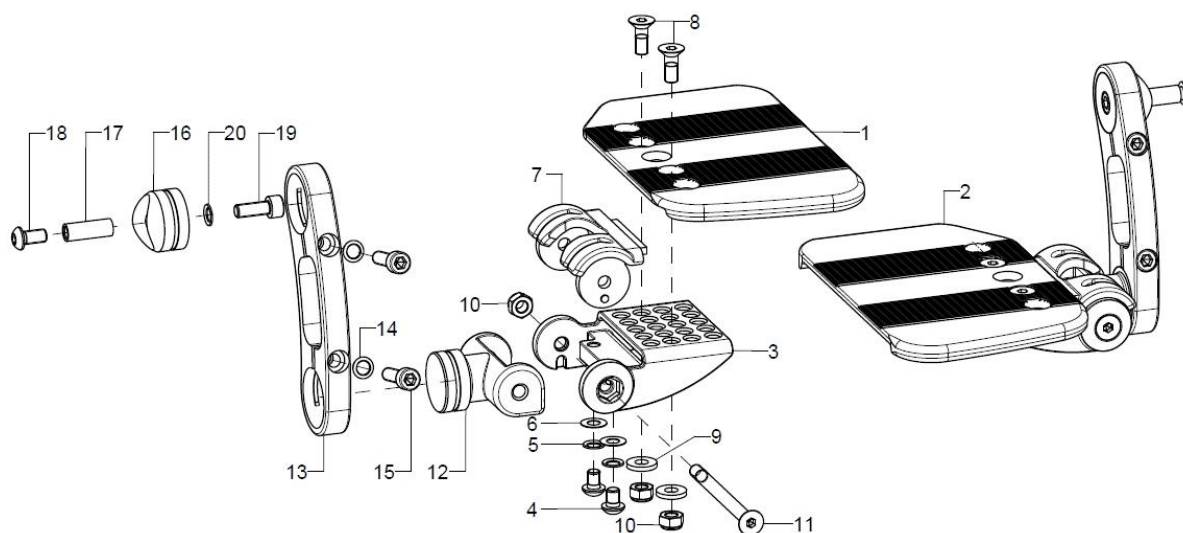


pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-20	1	403602	leg support divided, folds up sideways, inner attachment, complete		
1	1	40361000	foot plate 8mm divided right		40.36.10.XX.02.74
2	1	40361100	foot plate 8mm divided left		40.36.10.XX.02.74
3-21	2	40362002	inner attachment of divided leg support complete (adjustment part, locking part, joint, positioning part (length), connection cone, mounting set)		
3-7	2	40361020	adjustment unit divided incl. grid		40.36.10.30.00.74
4	2		lens head screw with hexagon socket ISO 7380 - M6x8		01.42.08.47.65.00
5	2		lock washer S6		02.99.10.60.01.10
6	2				02.68.06.00.30.30
7	2	40360102	raster part for foot plate		40.36.01.02.00.74
8-11	2	40360299	mounting set adjustment piece		XXXXXXXXXXXX
8	2		counter sunk screw with hexagon socket DIN 7991 M6x16		01.65.08.52.02.40
9	2		washer reinforced Ø14/6,2x2		58.30.15.01.00.24
10	3		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20
11	1		counter sunk screw with hexagon socket DIN 7991 M6x75		01.65.08.67.02.40
12	2	40360400			40.36.04.00.00.74
13	2	40390101	positioning part 12x125x41,9		40.39.01.01.00.69
altern.	2	40390110	positioning part 12x175x42,1		40.39.01.10.00.69
14-15	2	40390199	mounting set positioning part		
14	2		lock washer S6		02.99.10.60.01.10
15	2		cylinder screw DIN 912 M6x25		01.22.08.56.02.40
16	2	40390103	foot plate attachment		40.39.01.03.00.69
17-20	2	40390399	mounting set for foot plate attachment		
17	1				59.95.02.13.00.10
18	1		lens head screw with hexagon socket ISO 7380 - M6x10		01.42.08.48.65.00
19	1		lens head screw with hexagon socket ISO 7380 - M6x12		01.42.08.49.65.00
20	1		lock washer S6		02.99.10.60.01.10

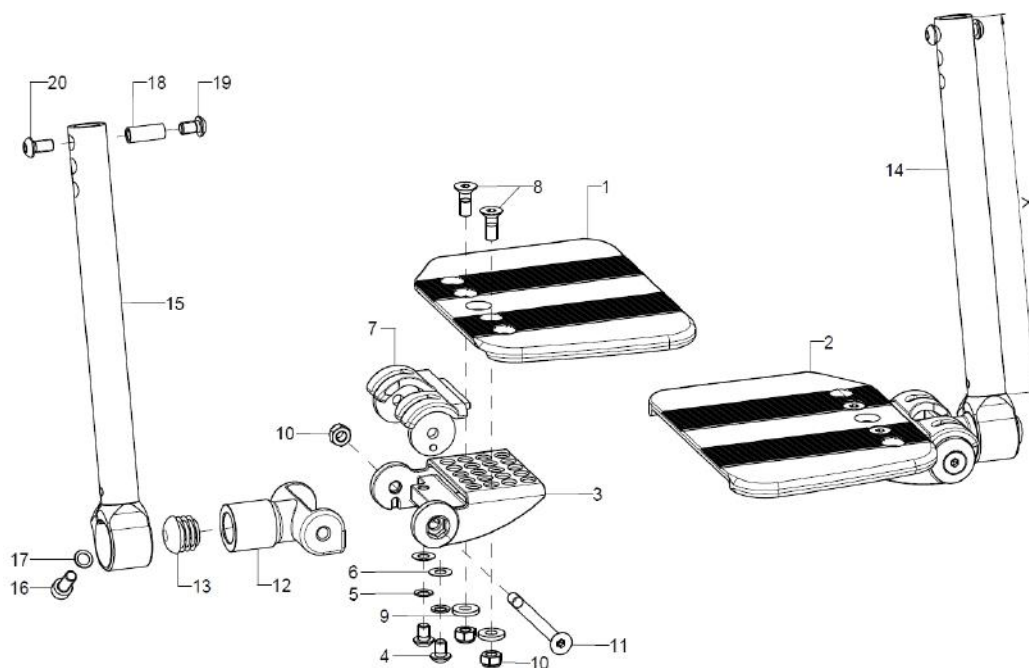Leg support divided, hinged sideways, external mounting 404003


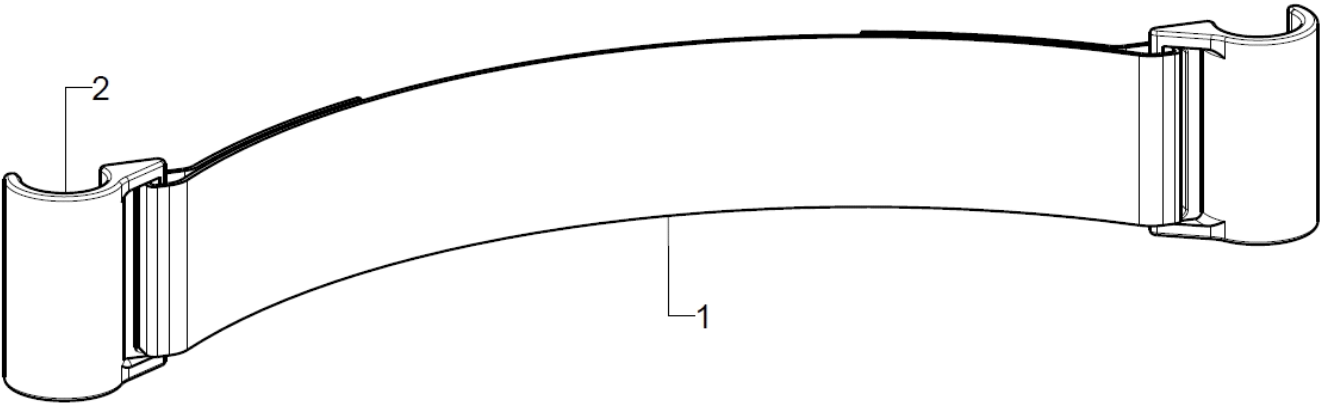
pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-20	1	403612	leg support divided, folds up sideways, outer attachment, complete		XXXXXXXXXXXX
1	1	40361000	foot plate 8mm divided right		40.36.10.XX.02.74
2	1	40361100	foot plate 8mm divided left		40.36.10.XX.02.74
3-7	2	40361020	adjustment unit divided incl. grid		40.36.10.30.00.74
4	2		lens head screw with hexagon socket ISO 7380 - M6x8		01.42.08.47.65.00
5	2		lock washer S6		02.99.10.60.01.10
6	2				02.68.06.00.30.30
7	1	40360102	raster part for foot plate		40.36.01.02.00.74
8-11	2	40360299	mounting set adjustment piece		XXXXXXXXXXXX
8	2		counter sunk screw with hexagon socket DIN 7991 M6x16		01.65.08.52.02.40
9	2		washer reinforced Ø14/6,2x2		58.30.15.01.00.24
10	3		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20
11	1		counter sunk screw with hexagon socket DIN 7991 M6x75		01.65.08.67.02.40
12-13	2	40360205	joint for divided leg support incl. plug		40.36.10.31.00.74
13	1		plugs for tube Ø25x1-2, black		04.63.10.25.00.10
14+16	1	40360121	leg support tube glued 235 mm left, incl. mounting screws		40.36.01.15.21.74
o.B.	1	40360131	leg support tube glued 305 mm left, incl. mounting screws		40.36.01.15.31.74
15+16	1	40360122	leg support tube glued 235 mm right, incl. mounting screws		40.36.01.15.22.74
o.B.	1	40360132	leg support tube glued 305 mm right, incl mounting screws		40.36.01.15.32.74
16	1		cylinder screw DIN 912 M6x16		01.22.08.52.62.40
17	1		lock washer S6		02.99.10.60.01.10
18-20	2	40390299	mounting set leg support tube		40.39.02.99.90.00
18	1		bushing M6x19,5		40.39.01.04.50.69
19	1		lens head screw with hexagon socket ISO 7380 - M6x10		01.42.08.48.65.00
20	1		lens head screw with hexagon socket ISO 7380 - M6x12		01.42.08.49.65.00

Heel cup

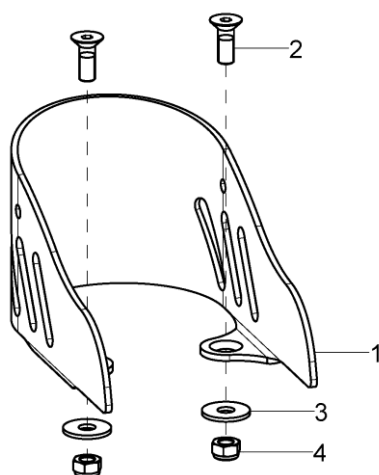


pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-4	2	402001	heel holder size 1 7x11,5 cm incl. mounting set		40.20.01.01.00.14
o.B.	2	402002	heel holder size 2 9x12,5 cm incl. mounting set		40.20.02.01.00.14
o.B.	2	402003	heel holder size 3 10x15,5 cm incl. mounting set		40.20.03.01.00.14
2	2		counter sunk screw with hexagon socket DIN 7991 M5x14		01.65.07.90.05.00
3	2		washer DIN 9021 - M5		02.79.05.31.20.20
4	2		hexagon nut self-locking low form DIN 985 - M5		02.32.00.50.01.20

Calf belts



pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-2	1	4041			
1	1	404108			40.41.03.XX.00.10
2	2	404106			40.41.06.01.00.10


Seat cover + cover carrier

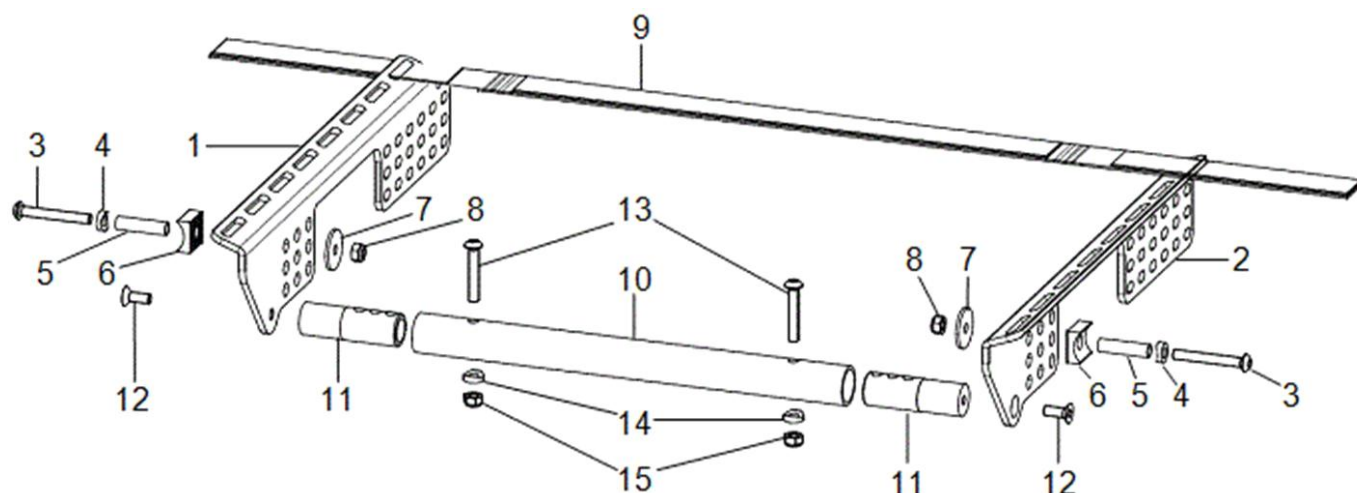

pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1	1	191301ST	cover support right		19.13.01.00.02.69
o.B.	1	191305ST			19.13.02.00.02.69
2	1	191302ST	cover support left		19.13.01.00.01.69
o.B.	1	191304ST			19.13.02.00.01.69
3-8	2	19080299	mounting set support angle		XXXXXXXXXXXX
3	1		lens head screw with hexagon socket ISO 7380 - M6x45		01.42.08.61.65.00
4	1				40.04.16.01.00.10
5	1				19.08.01.01.00.69
6	1				76.01.03.01.00.10
7	1				40.24.01.04.00.54
8	1		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20
9	5-10	191303ST	seat straps		19.13.03.00.00.69
10	1	5834SB	crossbar D=24x2xSW		58.34.01.00.00.69
11-15	2				
11	2	583402	crossbar adapter, RD 20x64		58.34.01.06.00.69
12	1		counter sunk screw with hexagon socket DIN 7991 M6x16		01.65.08.52.02.40
13-15	2	583499	mounting set crossbar on frame		XXXXXXXXXXXX
13	1				01.42.08.59.62.60
14	1				40.04.16.01.00.10
15	1		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20

*Bei den Rahmengrößen 1 + 2 wird der Bespannungsträger vorne mit dem Seitenteil verschraubt.
Der Bespannungsträger wird hinten mit der Lochplatte verschraubt.

SeatFix + SeatFix carrier

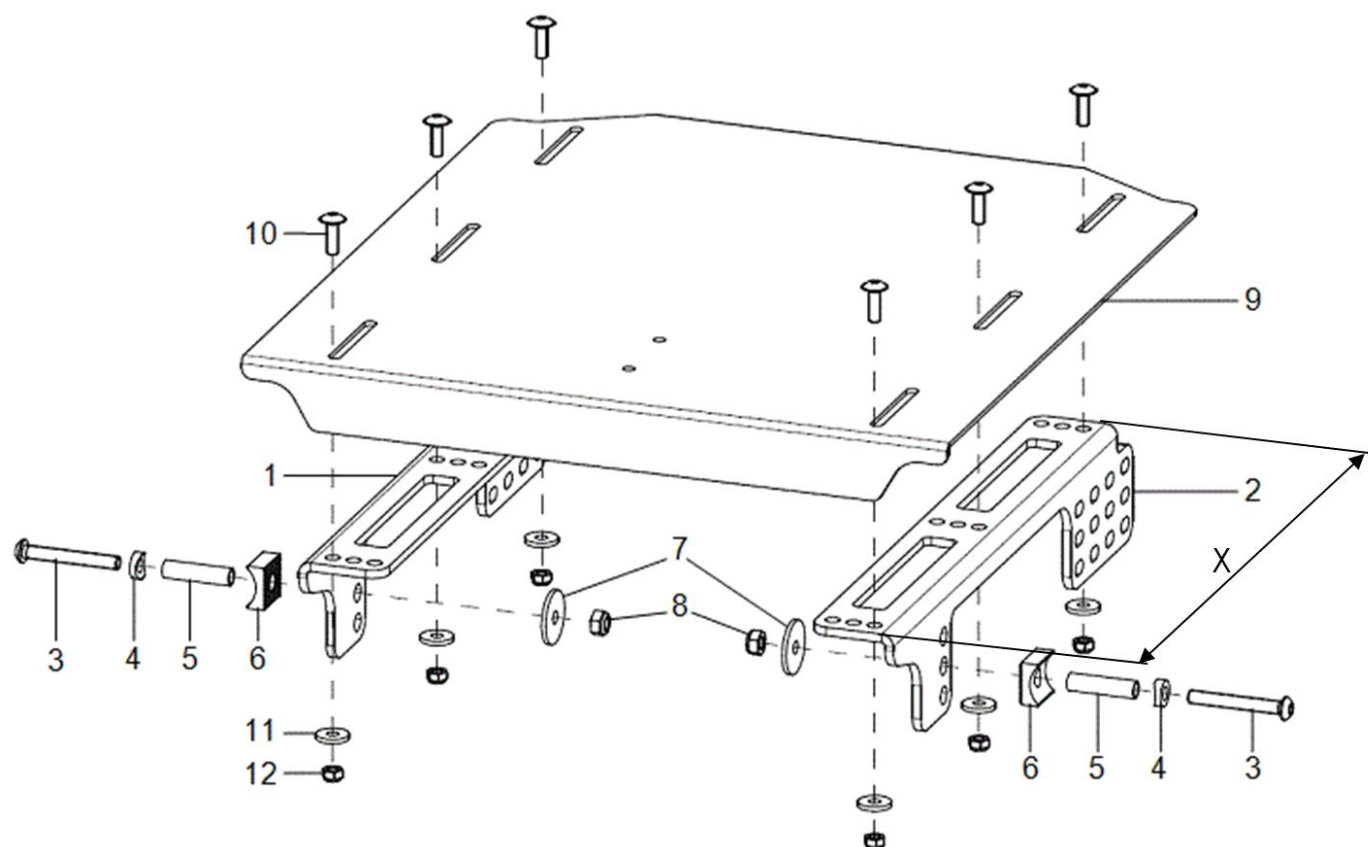

pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1	1	1908XX01	seat plate support right		19.08.02.00.02.69
o.B.	1	190803ST			19.08.02.03.02.81
2	1	1908XX02	seat plate support left		19.08.02.00.01.69
o.B.	1	190802ST			19.08.02.30.01.81
3-8	2	19080299	mounting set support angle		XXXXXXXXXXXX
3	1		lens head screw with hexagon socket ISO 7380 - M6x45		01.42.08.61.65.00
4	1				40.04.16.01.00.10
5	1				19.08.01.01.00.69
6	1				76.01.03.01.00.10
7	1				40.24.01.04.00.54
8	1		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20
9	1	19080200	seat plate		19.08.02.00.00.69
10-12	1	19080099	mounting set seat lug		XXXXXXXXXXXX
10	6		lens head screw with hexagon socket M5x20		01.43.07.92.05.00
11	6		washer DIN 7349 - 5,3		02.77.05.22.00.20
12	6		hexagon nut self-locking low form DIN 985 - M5		02.32.00.50.01.20

*Bei den Rahmengrößen 1 + 2 wird der Sitzplattenträger vorne mit dem Seitenteil verschraubt.
Der Sitzplattenträger wird hinten mit der Lochplatte verschraubt.


Seat cushion

pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
o.B.	1	891002	anatomical seat cushion		XXXXXXXXXXXXX
o.B.	1	891003	cover for anatomical seat cushion		XXXXXXXXXXXXX
o.B.	1	891004	incontinence cover for anatomical seat cushion		XXXXXXXXXXXXX
o.B.	1	8003	seat cushion, synthetic foam, 3cm thick		XXXXXXXXXXXXX
o.B.	1	8005	seat cushion, synthetic foam, 5cm thick		XXXXXXXXXXXXX
o.B.	1	8103	seat cushion, memory foam, 3cm thick		XXXXXXXXXXXXX
o.B.	1	8105	seat cushion, memory foam, 5cm thick		XXXXXXXXXXXXX
o.B.	1	80XX	cover for seat cushion		XXXXXXXXXXXXX
o.B.	1	8603	incontinence protection for 3cm cushion (non refundable)		
o.B.	1	8605	incontinence protection for 5cm cushion (non refundable)		

Stoffmuster	XX
Betex schwarz	20
Betex blau	21
Betex rot	22
Betex grau	23
Betex grün	24
Betex gelb	25
Betex orange	27


Alle Stoffmuster können unter www.sorgrollstuhltechnik.de angesehen werden.

Abduction wedge

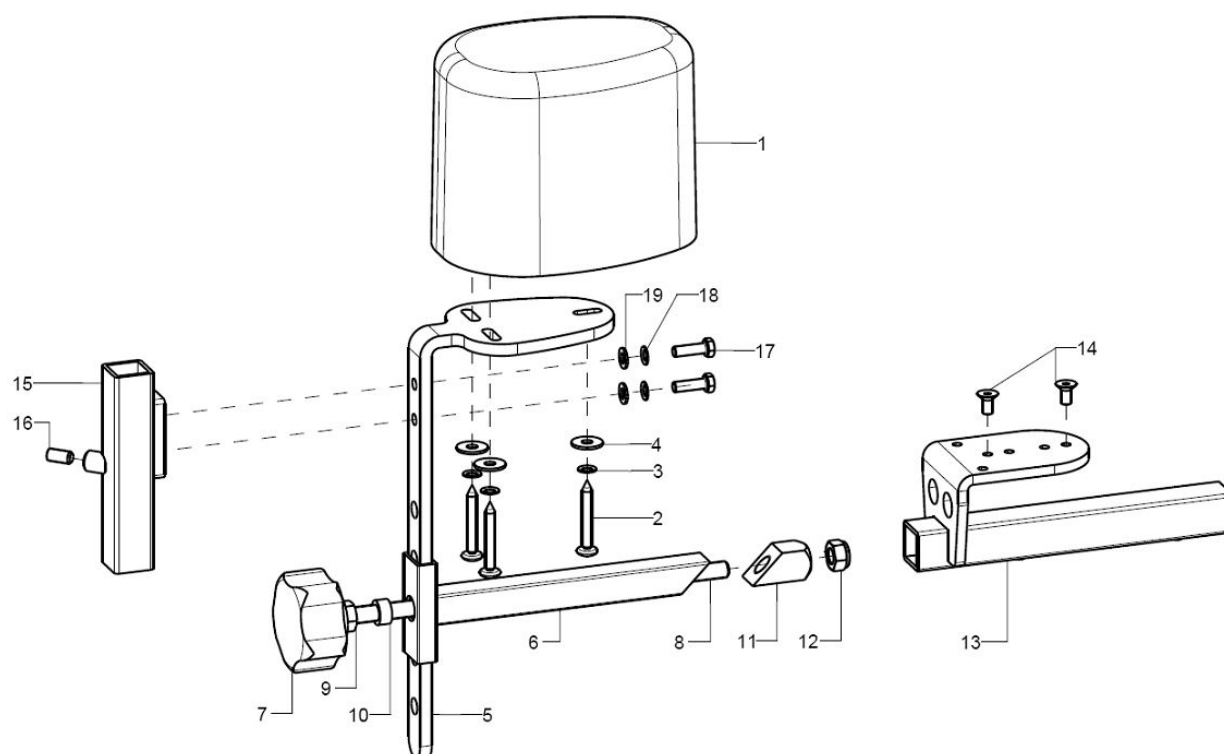


pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-14	1	1952	abduction wedge size s		XXXXXXXXXXXX
o.B.	1	1953	abduction wedge size m		XXXXXXXXXXXX
o.B.	1	1954	abduction wedge size l		XXXXXXXXXXXX
1	1	195002	abduction wedge small		04.65.17.09.01.20
o.B.	1	195003	abduction wedge medium		04.65.17.11.03.20
o.B.	1	195004	abduction wedge large		04.65.17.13.02.20
2-4	1	19500299	mounting set abduction wedge on holder angle		XXXXXXXXXXXX
2	3				01.88.07.99.00.80
3	3		lock washer S5		02.99.10.50.01.10
4	3		washer DIN 9021 - M5		02.79.05.31.20.20
5	1	195101	bracket for abduction wedge		19.50.03.02.00.10
6	1	195103	weldment clamber		19.50.02.01.00.10
7-12	1	684004	clamping set for posterior truss pad or abduction wedge		XXXXXXXXXXXX
7	1		5-star knob ø50 - M8 internal thread, black		04.05.35.34.00.40
8	1				
9	1		hexagon nut DIN 934 - M8		02.19.00.80.01.20
10	1				28.00.31.01.00.01
11	1				68.40.04.04.00.51
12	1		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20
13-14	1	195102	weldment holder abduction wedge		19.50.01.01.00.10
14	2		counter sunk screw with hexagon socket DIN 7991 M5x10		01.65.07.88.05.00
15-19	1	400105	universal leg support socket for combination abduction wedge with therapy table incl. mounting set		40.01.01.05.00.28
16	1		setscrew with hexagon socket DIN 913, M6x12		01.67.08.49.61.20
17	2		hexagon screw DIN 933 - M5x16		01.04.07.92.02.40
18	2		lock washer S5		02.99.10.50.01.10
19	2		washer DIN 125 - A 5,3		02.61.05.31.01.20

Therapy table, version 1, hinged sideways

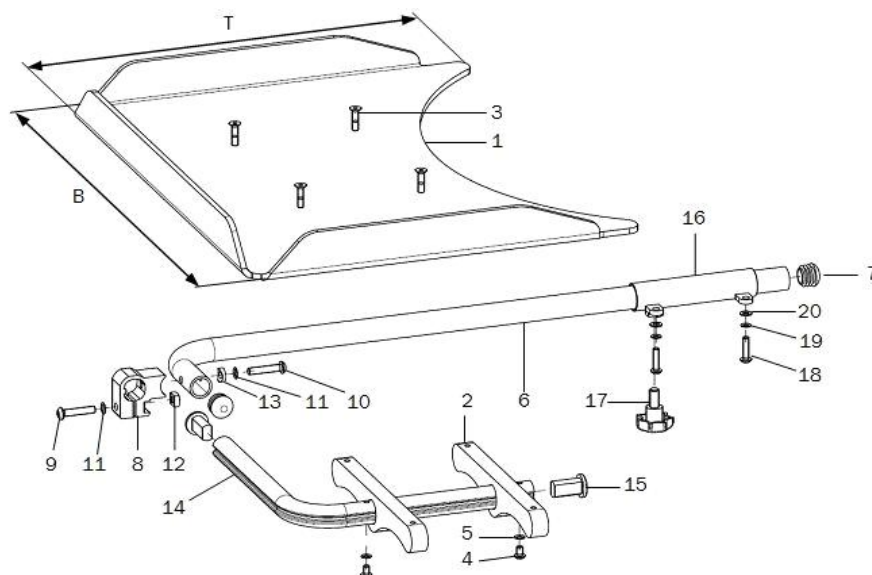


pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-20	1	9204XX	therapy table version 1 swingaway to the side complete		XXXXXXXXXXXX
1	1	920030	therapy table size 1 (B45xT36) polycarbonate		92.00.10.01.00.51
o.B.	1	920031	therapy table size 2 (B59xT45) polycarbonate		92.00.11.01.00.51
o.B.	1	920032	therapy table size 3 (B70xT50) polycarbonate		92.00.12.01.00.51
o.B.	1	920033	therapy table size 1 (B50xT40) wood		92.00.13.01.00.10
o.B.	1	920034	therapy table size 2 (B60xT50) wood		92.00.14.01.00.10
2-20	1	9204	mounting set for therapy table version 1 complete without table		
2	2				
3-5	1				XXXXXXXXXXXX
3	4		counter sunk screw with hexagon socket DIN 7991 M5x20		01.65.07.94.05.00
4	2		lens head screw with hexagon socket ISO 7380 - M5x8		01.42.07.87.65.00
5	2		lock washer S5		02.99.10.50.01.10
6	1	920402	holding tube, incl. plug		92.04.02.00.00.10
7	2		plugs for tube ø20x1-2, black		04.63.10.20.00.10
8-13	1	92040403	clamp unit incl. mounting set		92.04.04.03.00.10
9-13					XXXXXXXXXXXX
9	1		lens head screw with hexagon socket ISO 7380 - M6x30		01.42.08.58.65.00
10	1				01.42.08.59.62.60
11	2		lock washer S6		02.99.10.60.01.10
12	1		square nut DIN 557 M6		02.13.00.60.01.20
13	1				40.04.16.01.00.10
14-15	1	920404	foldable bail tube incl tube end plug		92.04.04.00.00.10
15	2	620105	plug for profile tube		62.01.05.00.00.74
16-20	1	920405	tube adapter, weldment, right incl. mounting set		92.07.01.01.02.10
o.B.	1	920406	tube adapter, weldment, left incl. mounting set		92.07.01.01.01.10
o.B.	2	920407	tube adapter, weldment incl. mounting set		92.04.05.01.00.10
17-20	2	92040599	mounting set tube intake		XXXXXXXXXXXX
17	1		7-star knob ø32 - M8x15 external thread, black		04.05.47.14.15.40
18	2		lens head screw with hexagon socket ISO 7380 - M5x20		01.42.07.94.65.00
19	2		lock washer S5		02.99.10.50.01.10
20	2		washer DIN 125 - A 5,3		02.61.05.31.01.20

Therapy table, version 2, fixed to the arm pad

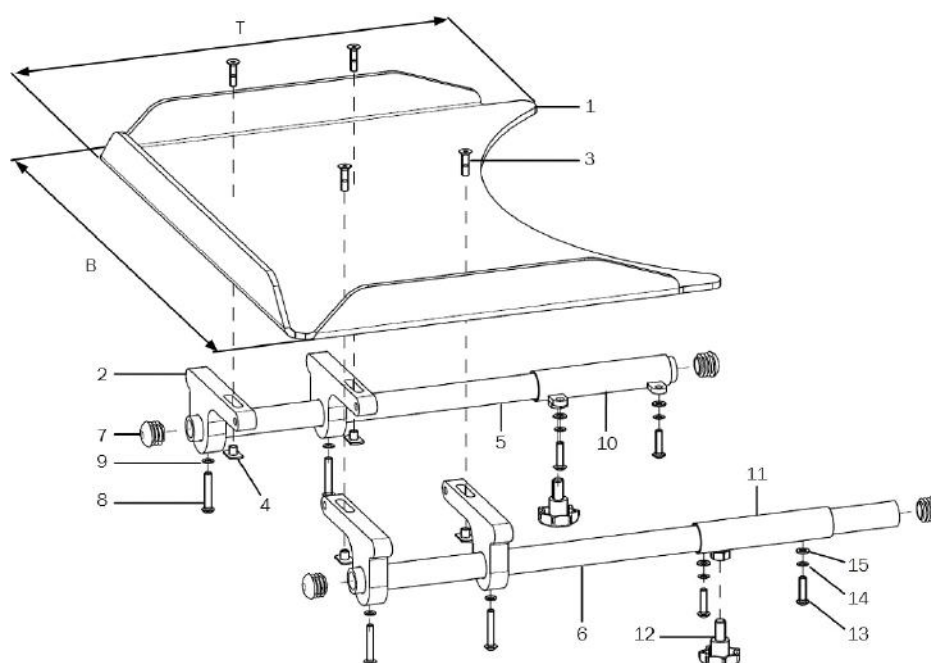


pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-15	1	9207XX	therapy table version 2 on arm pad complete		XXXXXXXXXXXX
1	1	920030	therapy table size 1 (B45xT36) polycarbonate		92.00.10.01.00.51
o.B.	1	920031	therapy table size 2 (B59xT45) polycarbonate		92.00.11.01.00.51
o.B.	1	920032	therapy table size 3 (B70xT50) polycarbonate		92.00.12.01.00.51
o.B.	1	920033	therapy table size 1 (B50xT40) wood		92.00.13.01.00.10
o.B.	1	920034	therapy table size 2 (B60xT50) wood		92.00.14.01.00.10
2-15	2	9207	mounting set for therapy table version 2 on arm rest		
2	4	920705	pluggable supporting frame 15x61x120		92.07.05.01.00.51
3-4	1	92070599	mounting set support bracket		XXXXXXXXXXXX
3	4		counter sunk screw with hexagon socket DIN 7991 M5x20		01.65.07.94.05.00
4	4		weld nut C- M5x7,5		02.49.33.03.00.10
5+7-9	1	920702	holder tube 20x3x400 incl. mounting set		92.07.02.02.00.29
6-9	1	920704	holder tube 20x3x450 incl. mounting set		92.07.02.04.00.29
7	2		plugs for tube ø20x1-2, black		04.63.10.20.00.10
8	2		lens head screw with hexagon socket ISO 7380 - M5x30		01.42.07.98.65.00
9	2		lock washer S5		02.99.10.50.01.10
10+12-15	1	920701	tube adapter, weldment, right incl. mounting set		92.07.01.01.02.10
11-15	1	920703	tube adapter, weldment, left incl. mounting set		92.07.01.01.01.10
o.B.	2	920407	tube adapter, weldment incl. mounting set		92.04.05.01.00.10
12-15	2	92040599	mounting set tube intake		XXXXXXXXXXXX
12	1		7-star knob ø32 - M8x15 external thread, black		04.05.47.14.15.40
13	2		lens head screw with hexagon socket ISO 7380 - M5x20		01.42.07.94.65.00
14	2		lock washer S5		02.99.10.50.01.10
15	2		washer DIN 125 - A 5,3		02.61.05.31.01.20

Therapy table, version 6, mounted under seat plate

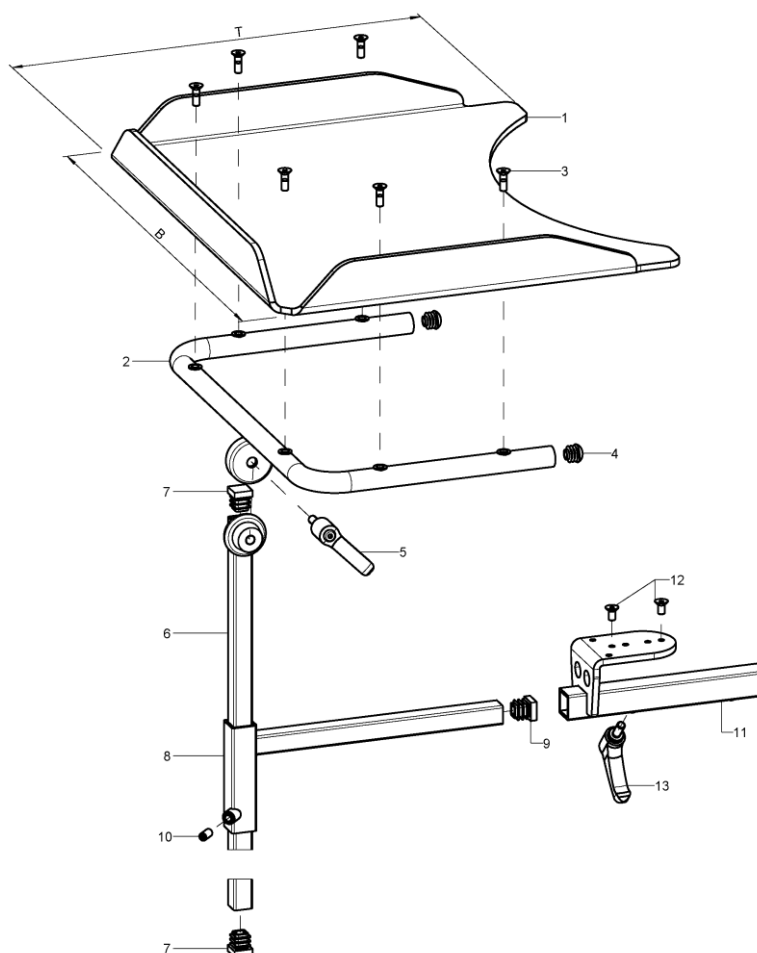


pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-13	1	9209XX	therapy table central mounting under SeatFix complete		XXXXXXXXXXXX
1-7	1				
1	1	920030	therapy table size 1 (B45xT36) polycarbonate		92.00.10.01.00.51
o.B.	1	920031	therapy table size 2 (B59xT45) polycarbonate		92.00.11.01.00.51
o.B.	1	920032	therapy table size 3 (B70xT50) polycarbonate		92.00.12.01.00.51
o.B.	1	920033	therapy table size 1 (B50xT40) wood		92.00.13.01.00.10
o.B.	1	920034	therapy table size 2 (B60xT50) wood		92.00.14.01.00.10
2-13	1	920909	mounting set therapy table for centre attachment (U-holder, cone rod with clamp lever, adapter holder, square adapter)		
2-5	1	920901	holder for therapy table incl. mounting set		92.09.01.01.00.72
3	6		counter sunk screw with hexagon socket DIN 7991 M5x16		01.65.07.92.05.00
4	2		plugs for tube ø22x1-2, black		04.63.10.22.00.10
5	1	650050	clamp lever adjustable M8x16, black		04.13.10.80.16.50
6-7	1	920902	cone rod 450 incl. plugs		92.08.03.04.00.24
7	2		plugs for square tubes, black 15x15		04.63.11.15.00.10
8-10	1	920903	adapter 220x85 incl. mounting set		92.08.02.01.00.24
9	1		plugs for square tubes, black 15x15		04.63.11.15.00.10
10	1		setscrew with hexagon socket DIN 913, M6x12		01.67.08.49.61.20
11-13	1	195102	weldment holder abduction wedge		19.50.01.01.00.10
12	2		counter sunk screw with hexagon socket DIN 7991 M5x10		01.65.07.88.05.00
13	1		clamp lever adjustable M6x16, black		04.13.10.60.16.50

Back tube standard

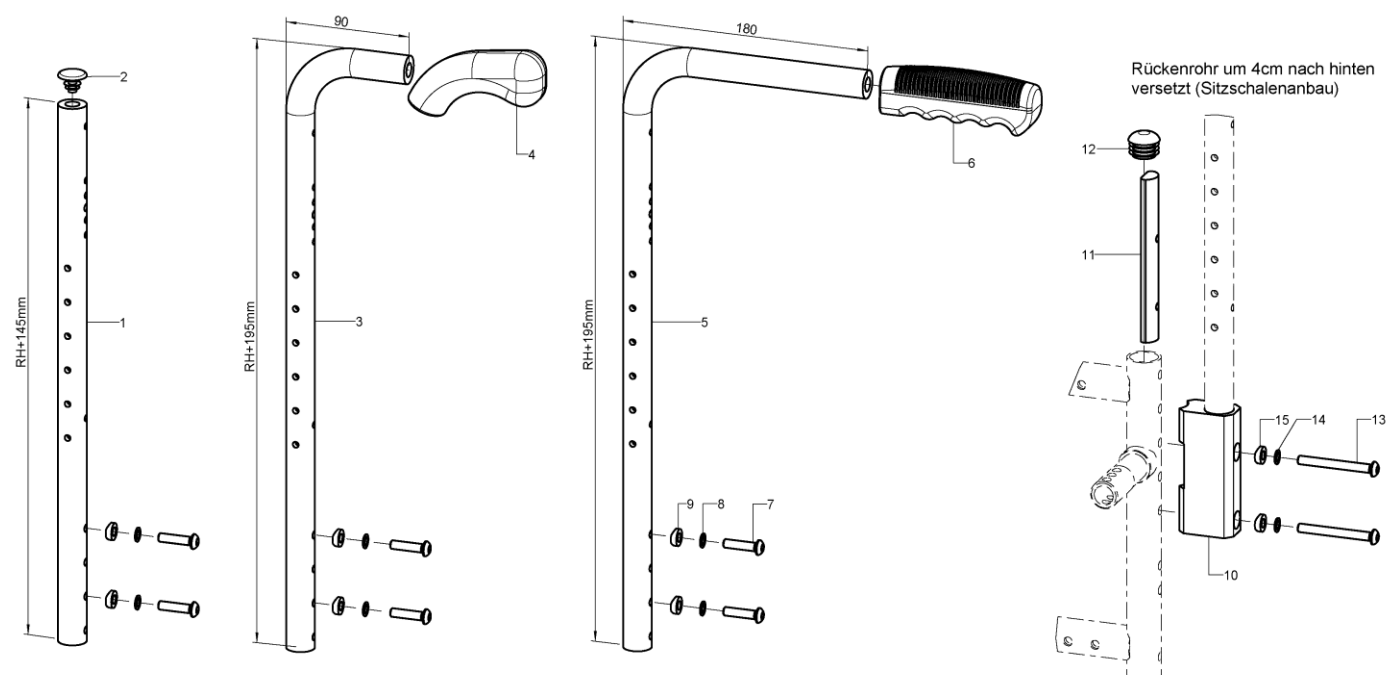

pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-2	2	623705XX			62.37.05.00.00.00
2	1				04.63.10.20.01.10
3	2	623806XX			62.37.06.01.00.00
4	2	650021	handle PVC soft, curved, ø20 mm		62.37.03.05.00.69
5	2	623706XX			62.37.06.02.00.00
6	2	650020	handle PVC black, straight without base		65.00.20.01.04.10
7-9	2	62370099	mounting set back tube		XXXXXXXXXXXX
7	2		lens head screw with hexagon socket ISO 7380 - M6x25		01.42.08.56.65.00
8	2		lock washer S6		02.99.10.60.01.10
9	2		saddle washer Ø12/6x2,5		35.04.16.01.00.10
10-15	2	6225			62.25.02.01.00.69
o.B.	2	622501			62.25.02.03.00.69
11	1				
12	1		plugs for tube ø22x1-2, black		04.63.10.22.00.10
13	2(1)		lens head screw with hexagon socket ISO 7380 - M6x55		01.42.08.63.65.00
o.B.	1				01.22.08.64.02.40
14	2		lock washer S6		02.99.10.60.01.10
15	2		saddle washer Ø12/6x2,5		35.04.16.01.00.10


Back, adjustable in angle

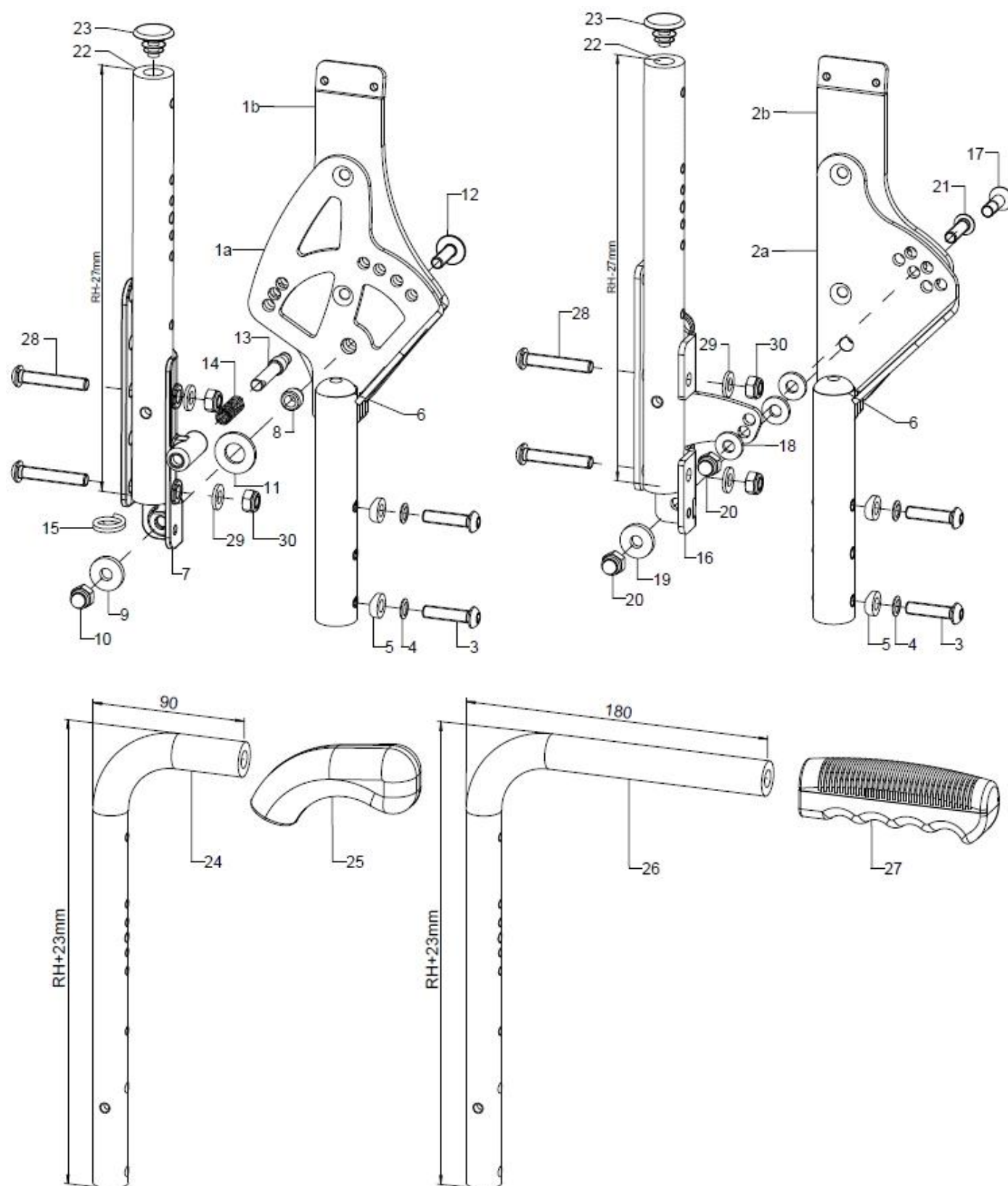



pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1a	1	62370202	back angle adjustment, right		61.09.01.04.02.69
o.B.	1	62370201	back angle adjustment, left		61.09.01.04.01.69
1b	1	61110202			61.11.02.00.02.75
o.B.	1	61110201			61.11.02.00.01.75
2a	1	61110102	angle adjustment, right		61.11.01.00.02.74
o.B.	1	61110101	angle adjustment, left		61.11.01.00.01.74
2b	1	61110104			61.11.01.00.02.75
o.B.	1	61110103			61.11.01.00.01.75
3-6	2	62370299	mounting set back angle adjustment		XXXXXXXXXXXX
3	2		lens head screw with hexagon socket ISO 7380 - M6x25		01.42.08.56.65.00
4	2		lock washer S6		02.99.10.60.01.10
5	2		saddle washer Ø12/6x2,5		35.04.16.01.00.10
6	1	620002	plug for rod ø20x1-2, black		04.63.10.20.00.10
7-12	1	62370204	U-part 137 mm, right		61.09.05.01.02.00
o.B.	1	62370203	U-part 137 mm, left		61.09.05.01.01.00
8-12	2	62370298	mounting set U-part 137 mm		XXXXXXXXXXXX
8	1				61.03.02.01.00.18
9	1		washer DIN 7349 - 6,4		02.77.06.53.00.20
10	1		hexagon cap nut low form with clamp DIN 986 - M6		02.33.00.60.11.20
11	1		washer DIN 125 - A 10,5 - PA black		02.61.10.52.01.71
12	1		flat head screw with hexagon socket type KF - M6x20		01.67.08.54.60.80
13-15	2	610109	locking bolt incl. mounting set (spring and ring)		61.01.02.04.00.24
14	2		pressure spring DIN 2098 0,7x7,65/9,05x21		04.21.00.80.21.00
15	2		wire form ring 2x16		04.29.04.42.20.00
o.B.	1	610212	keyring, spring steel wire ø3 - outer ring ø 32 + polyester cord ø3 mm		Ohne Nummer
16	1	61110302	U-part for angle adjustment, right		61.11.01.03.02.74
o.B.	1	61110301	U-part for angle adjustment, left		61.11.01.03.01.74
17-21	2	61110399	mounting set U-part for angle adjustment		XXXXXXXXXXXX
17	1		counter sunk screw with hexagon socket DIN 7991 M6x20		01.65.08.54.02.40
18	3		washer reinforced Ø14/6,2x2		58.30.15.01.00.24
19	1		washer DIN 7349 - 6,4		02.77.06.53.00.20
20	2		hexagon cap nut low form with clamp DIN 986 - M6		02.33.00.60.11.20
21	1		lens head screw with hexagon socket ISO 7380 - M6x20		01.42.08.54.05.00
22-23	2	623302XX			62.33.02.00.00.00
23	2				04.63.10.20.01.10
24	2	623403XX			62.33.03.01.00.00
25	2	650021	handle PVC soft, curved, ø20 mm		62.37.03.05.00.69
26	2	623303XX			62.33.03.02.00.00
27	2	650020	handle PVC black, straight without base		65.00.20.01.04.10
28-30	2	62330099	mounting set back tube		XXXXXXXXXXXX
28	2		lens head screw with hexagon socket ISO 7380 - M6x35		01.42.08.59.65.00
29	2		washer DIN 125 - A 6,4		02.61.06.41.41.20
30	2		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20


Back rest with gas pressure spring adjustment

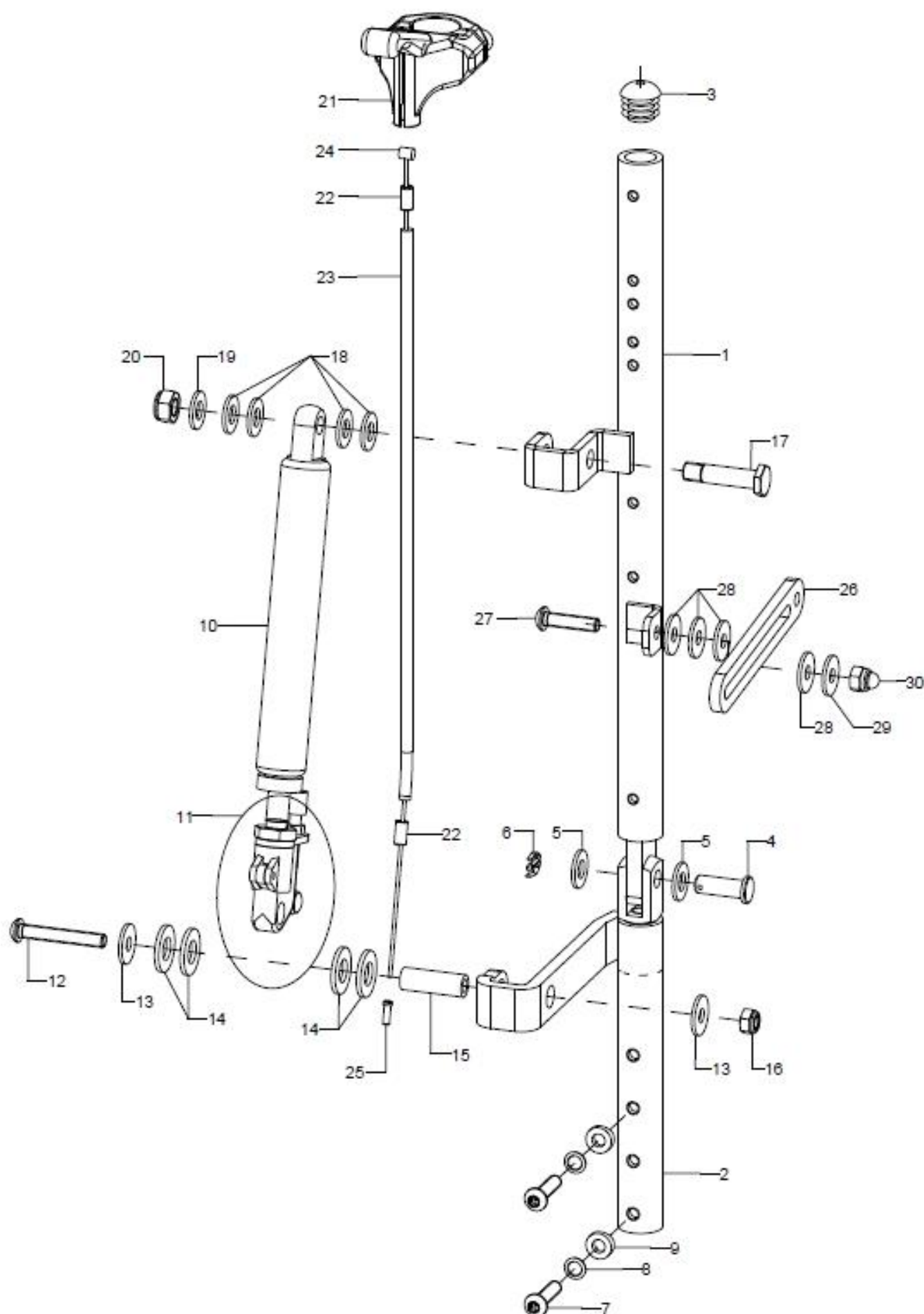



pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-30	1	6107			
1+3-5	1	6107XX02			61.07.06.02.XX.00
o.B.	1	6107XX01			61.07.06.01.XX.00
2	2	610710			61.07.10.01.00.00
3	1		plugs for tube ø18x1-2, black		04.63.10.18.00.10
4	1				03.56.08.19.51.00
5	2		washer DIN 125 - A 8,4		02.61.08.41.41.20
6	1				03.46.02.20.01.00
7-9	2	62370299	mounting set back angle adjustment		XXXXXXXXXXXX
7	2		lens head screw with hexagon socket ISO 7380 - M6x35		01.42.08.59.65.00
8	2		lock washer S6		02.99.10.60.01.10
9	2		saddle washer Ø12/6x2,5		35.04.16.01.00.10
10-11	2	190505			04.27.04.10.00.50
o.B.	2	190507	gas pressure spring 200/40 complete with release knob		04.27.04.20.00.60
11	2	190521	gas pressure spring release knob		04.27.00.99.81.00
12-16	2	19050599			
12	1		lens head screw with hexagon socket ISO 7380 - M6x40		01.42.08.60.65.00
13	2		washer DIN 9021 -6,4		02.79.06.41.60.20
14	4		washer DIN 125 - A 10,5 - PA black		02.61.10.52.01.71
15	1				62.36.01.07.00.10
16	1		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20
17-20	1	190503	fitted bolt M8x35 SW13		19.05.03.10.00.28
18	4		washer DIN 125 - A 8,4 - PA black		02.61.08.41.61.71
19	1		washer DIN 125 - A 8,4		02.61.08.41.41.20
20	1		hexagon nut self-locking low form DIN 985 - M8		02.32.00.80.01.20
21-25	2	190512	gas pressure spring release incl. mounting set		XXXXXXXXXXXX
21	2	190513	hand release lever for gas pressure spring, not lockable		04.54.92.12.20.10
22	2	352412	end bush metal ø5		04.54.91.50.10.41
23	2	352013	outer shell for Bowden cable black 150mm		04.54.91.21.53.00
24	2	190517	inner cable for gas pressure spring 1.3x2000		04.54.91.11.31.30
25	2	352005	end cap for inner Bowden cable ø1-1,5 mm		04.54.91.50.10.40
26-30	2	610703			61.07.03.01.00.10
27	1		lens head screw with hexagon socket ISO 7380 - M6x25		01.42.08.56.65.00
28	4		washer DIN 9021- 6,4- PA		02.79.06.41.60.70
29	1		washer DIN 9021 -6,4		02.79.06.41.60.20
30	1		hexagon nut DIN 934 - M6		02.19.00.60.01.20

Stabilizer bar, tripartite

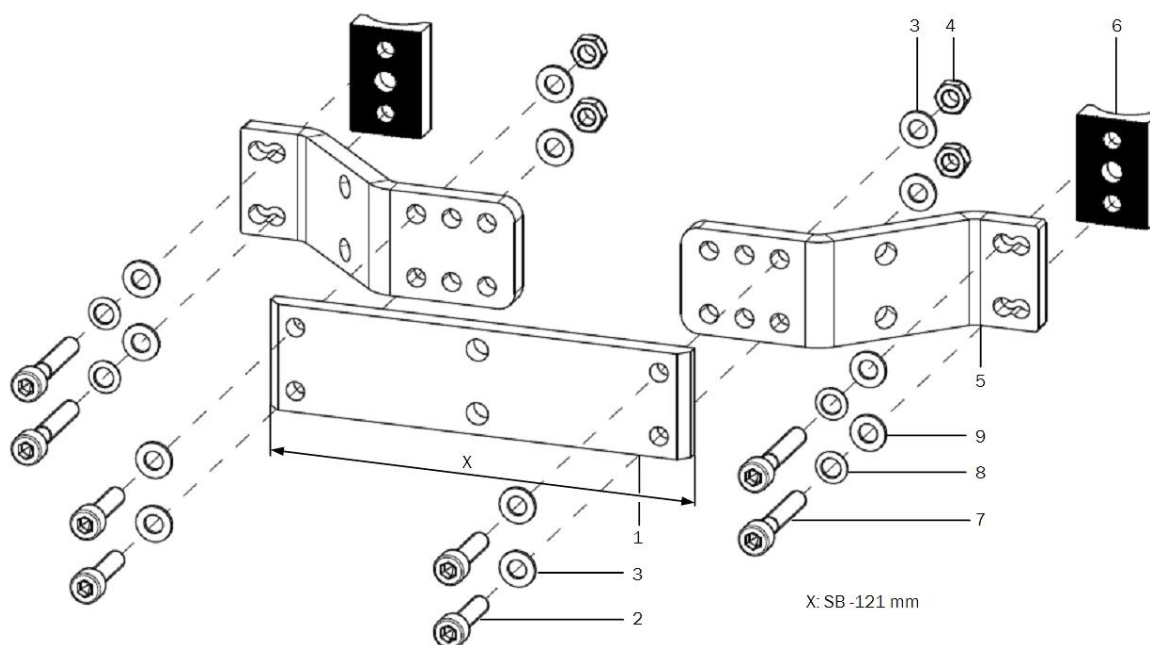


pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-5	1	640801	stabilizer bar 3-piece complete		XXXXXXXXXXXX
1	1	640802	traverse for stabilizer bar		64.08.01.00.00.69
2-4	1	64080099	mounting set for crossbar		XXXXXXXXXXXX
2	4		cylinder screw DIN 912 M5x20		01.22.07.94.72.40
3	8		washer DIN 125 - A 5,3		02.61.05.31.01.20
4	4		hexagon nut self-locking low form DIN 985 - M5		02.32.00.50.01.20
5	2	640805	adapter for stabilizer bar		64.08.01.01.00.69
6-9	2	64080599	mounting set adapter for stabilizer bar		XXXXXXXXXXXX
6	1	640102	half shell 30, R10, 20x8,5x30, 2xø5,2/18		64.01.02.03.00.29
7	2				01.22.07.98.62.40
8	2		lock washer S5		02.99.10.50.01.10
9	2		washer DIN 125 - A 5,3		02.61.05.31.01.20

Verwendung bei:


- Winkelverstellbarem Rücken bis RH30 ohne integrierte Schiebegriffe
- Winkelverstellbarem Rücken mit integrierten Schiebegriffen
- Rücken Standard mit integrierten Schiebegriffen
- Rücken Standard mit Einhandschiebegriff
- Winkelverstellbarem Rücken mit Einhandschiebegriff

Stabilizer bar, undivided

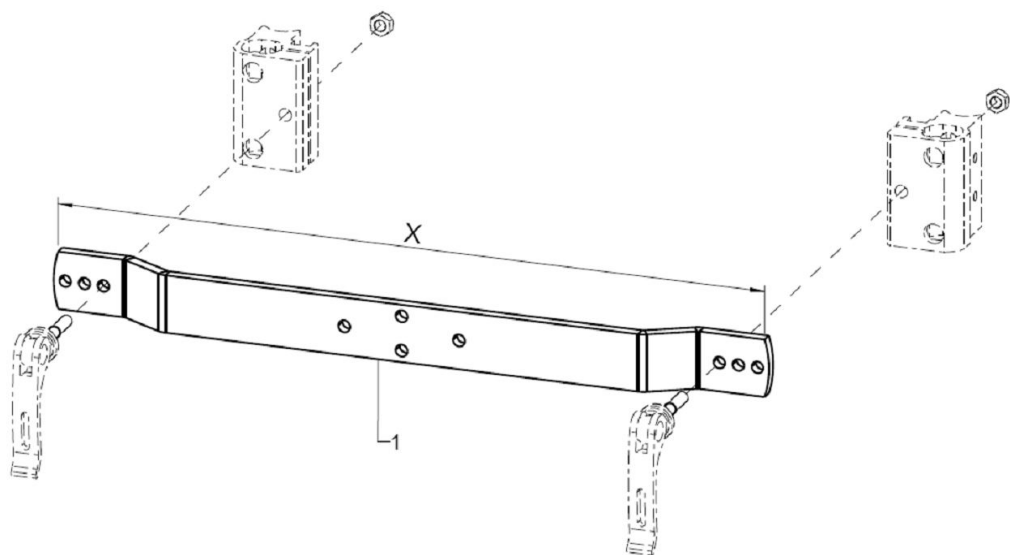

pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1	1	640901	stabilizer bar on SHG holder		64.08.00.00.00.69


Push handles, extendable

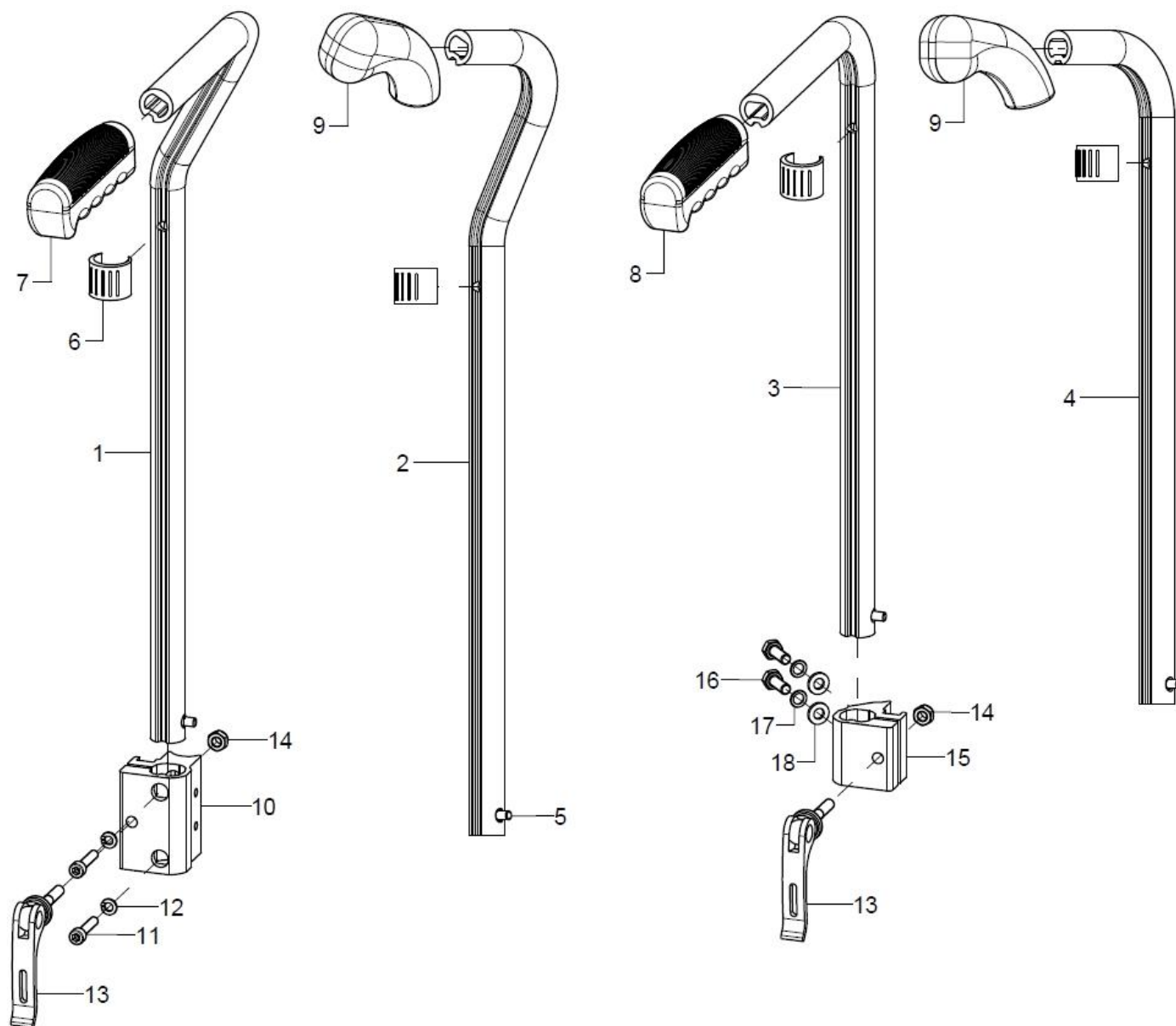



pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1+5-7	1	650532	push handle 15 cm, cranked, extendable, right complete		65.05.30.01.06.28
o.B.	1	650531	push handle 15 cm, cranked, extendable, left complete		65.05.30.01.05.28
altern.	1	650632	push handle 25 cm, cranked, extendable, right complete		65.06.30.01.06.28
o.B.	1	650631	push handle 25 cm, cranked, extendable, left complete		65.06.30.01.05.28
2+5-6+9	1	650552			65.05.50.01.02.00
o.B.	1	650551			65.05.50.01.01.00
altern.	1	650554			65.05.50.02.02.00
o.B.	1	650553			65.05.50.02.01.00
3+5-6+8	2	650130	push handle 15 cm, extendable straight, complete		65.01.30.01.00.10
o.B.	2	650230	push handle 25 cm, straight, extendable, complete		65.02.30.01.00.10
4-6+9	2	650251			65.02.50.01.00.00
o.B.	2	650250			65.02.50.02.00.00
5	2	650160	locking unit for push handles		65.01.30.03.00.10
6	2	046350200010	clamp conductor black, ø 18-20		04.63.50.20.00.10
7	1	650018	handle with bas right - 26x40x112 black		65.00.20.01.02.10
o.B.	1	650019	handle with bas left - 26x40x112 black		65.00.20.01.03.10
8	2	650017	handle PVC black, straight with base		65.00.20.01.01.10
9	2	650021	handle PVC soft, curved, ø20 mm		62.37.03.05.00.69
10-14	2	650051	push handle clamp holder 38x42x58, incl. eccentric clamp and mounting set		65.00.51.01.00.10
o.B.	2	650056			65.00.56.02.00.18
11	2		cylinder screw DIN 912 M5x25		01.22.07.96.62.40
12	2		spring ring DIN 127 - A 5		02.85.30.50.05.10
13	1	041340603510	eccentric clamp length 35 mm M6x20, black		04.13.40.60.35.10
14	1		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20
15-18+13-14	2	650053	S-handle clamp holder 45", on stabilizer bar		65.14.02.01.00.10
16	2		hexagon screw DIN 933 - M6x16		01.04.08.52.02.40
17	2		lock washer S6		02.99.10.60.01.10
18	2		washer DIN 125 - A 6,4		02.61.06.41.41.20


Push bar, extendable from (SW32)

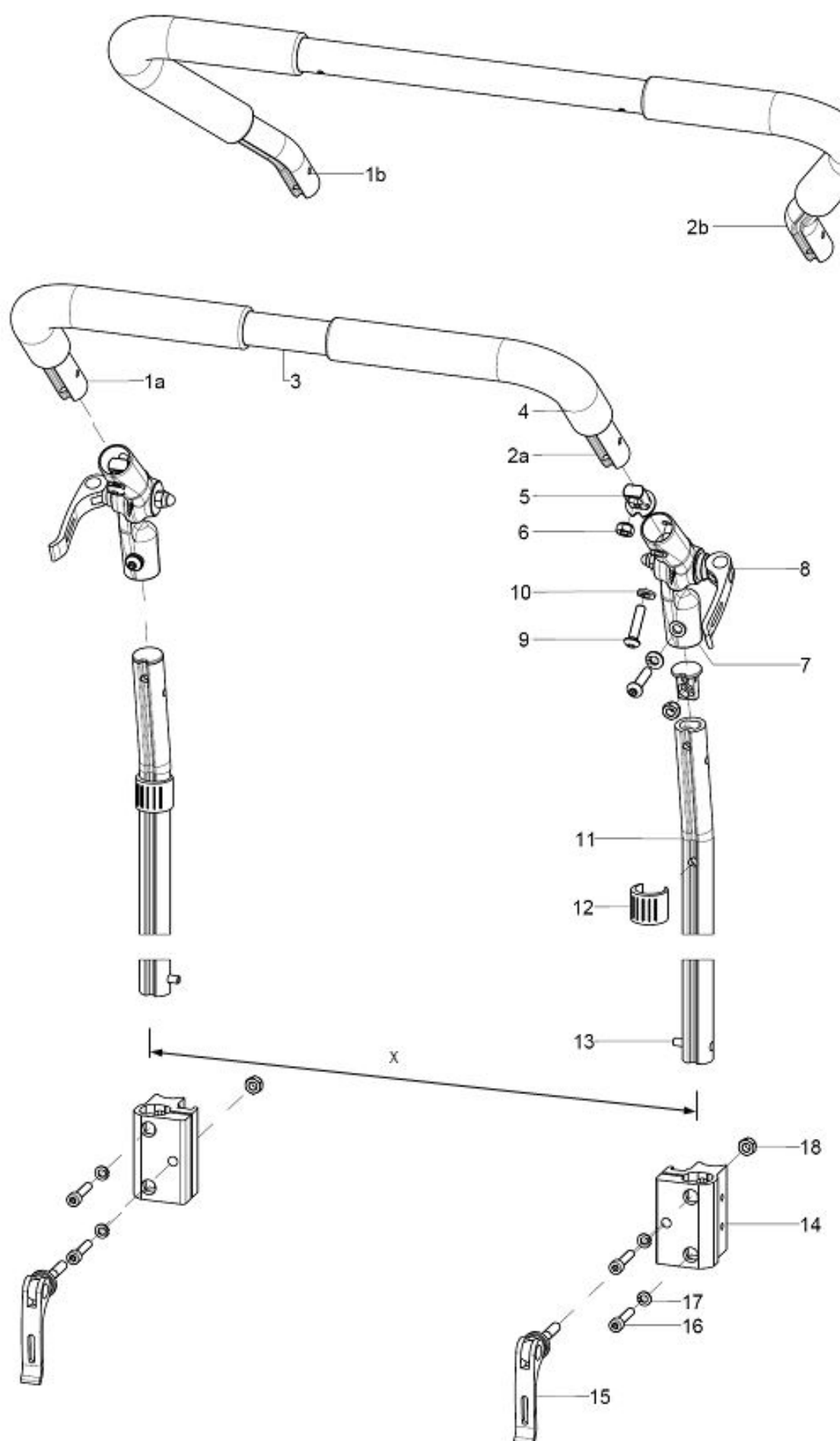





pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-5+9-11	1	6532	push bail extendable by 20 cm, 307° angle adjustment metal joint, complete		XXXXXXXXXXXXX
1a+2a+3-4	1	653001	push bail 90° pre-assembled for SW 40 – 46 when push handle clamp holder is mounted to the back tube		XXXXXXXXXXXXX
1b+2b+3-4	1	653002	push bail cropped pre-assembled		XX.XX.XX.XX.XX
1a	1	653009	push bail, bend part- 20x150x150, left		65.34.00.02.01.66
1b	1	653006	push bail left part		65.30.01.02.01.47
2a	1	653010	push bail, bend part- 20x150x150, right		65.34.00.02.02.66
2b	1	653007	push bail right part		65.30.01.02.02.47
3	1	653004	steel tube incl. riveting		65.30.01.04.00.24
4	2	653005	tube cover - rubber 260 mm		65.30.03.01.00.47
5-6	4	653406	plug for nut M6 incl hexagon nut		65.22.20.10.00.00
6	4		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20
7-10	2	652220	angle adjusting element metal 307° compl with eccentric clamp and mounting set		65.22.20.99.00.10
8	2	652224	eccentric clamp length 35 mm M6x20, black (incl. spring, nut, washer, bush)		04.13.40.60.35.10
9	2		lens head screw with hexagon socket ISO 7380 - M6x25		01.42.08.56.65.00
10	2		spring ring DIN 127- A6		02.85.30.60.05.10
11-13	2	653202	profile tube support 200		65.31.02.01.00.24
12	2	046350200010	clamp conductor black, ø 18-20		04.63.50.20.00.10
13	2	650160	locking unit for push handles		65.01.30.03.00.10
14-18	2	650061	push bail holder R10		65.00.51.02.00.10
15	1	041340603510	eccentric clamp length 35 mm M6x20, black		04.13.40.60.35.10
16	2		cylinder screw DIN 912 M5x25		01.22.07.96.62.40
17	2		spring ring DIN 127 - A 5		02.85.30.50.05.10
18	1		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20

Push bar, extendable (to SW30)



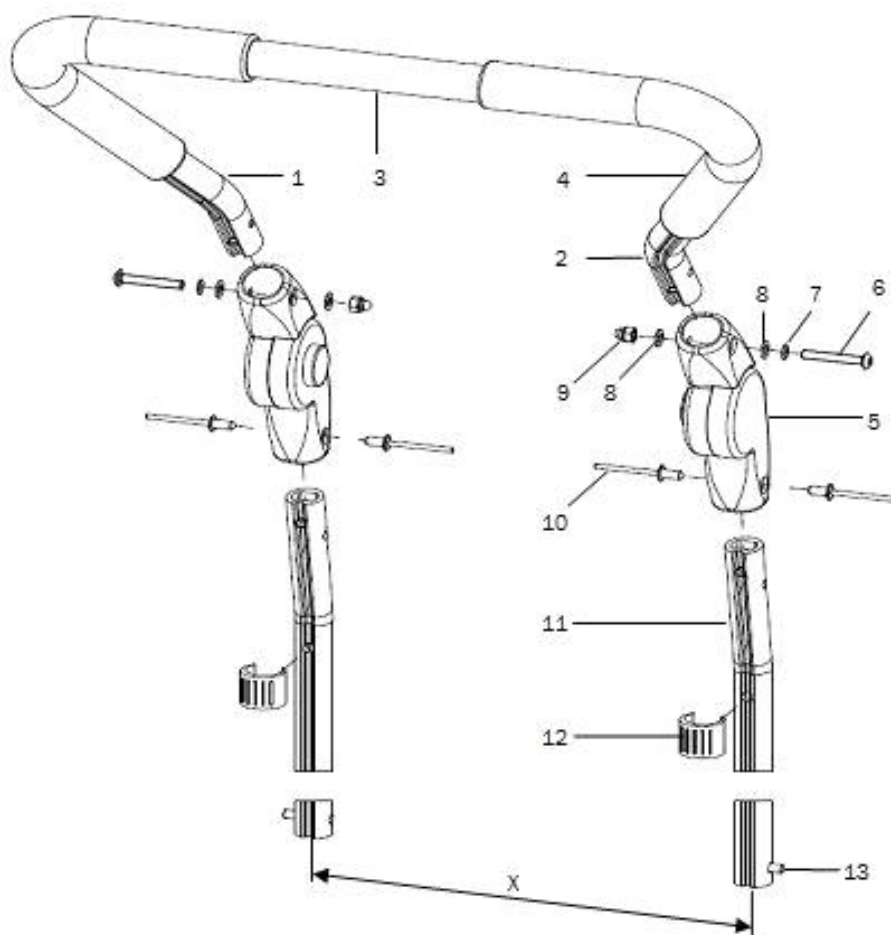


pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-13	1	6534	push bail extendable by 22 cm, 205° angle adjustment synthetic joint, complete		XXXXXXXXXXXX
1-4	1	653002	push bail cropped pre-assembled		XX.XX.XX.XX.XX
1	1	653006	push bail left part		65.30.01.02.01.47
2	1	653007	push bail right part		65.30.01.02.02.47
3	1	653004	steel tube incl. riveting		65.30.01.04.00.24
4	2	653005	tube cover - rubber 260 mm		65.30.03.01.00.47
5-10	2	653403	synthetic joint with push knob 205° incl. mounting set		04.03.01.20.01.00
6	1		lens head screw with hexagon socket ISO 7380 - M5x35		01.42.07.99.65.00
7	1		lock washer S5		02.99.10.50.01.10
8	2		washer DIN 125 - A 5,3		02.61.05.31.01.20
9	2		hexagon cap nut low form with clamp DIN 986 - M5		02.33.00.60.21.20
10	2		blind rivet F5x10 A2/A2		03.64.00.50.10.30
11	2	653202	profile tube support 200		65.31.02.01.00.24
12	2	046350200010	clamp conductor black, ø 18-20		04.63.50.20.00.10
13	2	650160	locking unit for push handles		65.01.30.03.00.10

Seat cover, adjustable

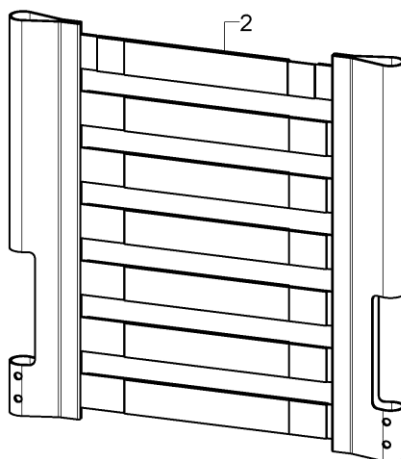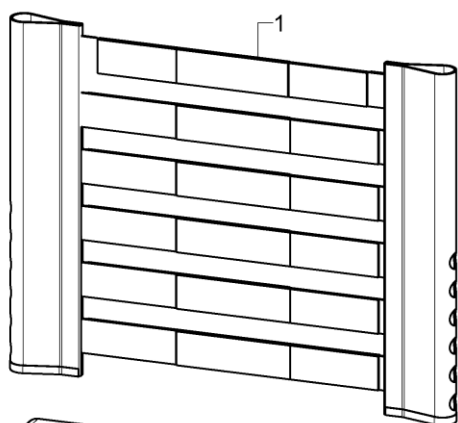


Stoffmuster	XX
Betex schwarz	20
Betex blau	21
Betex rot	22
Betex grau	23
Betex grün	24
Betex gelb	25
Betex orange	27

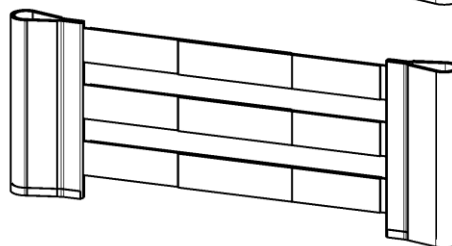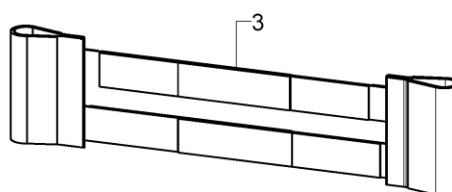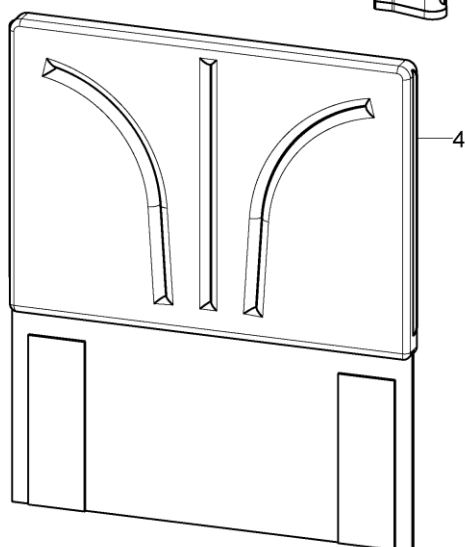

pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1+4	1	670102XX			XX.XX.XX.XX.XX
2+4	1	670103XX			XX.XX.XX.XX.XX
3+4	1	610105XX			XX.XX.XX.XX.XX
1	1				XX.XX.XX.XX.XX
2	1				XX.XX.XX.XX.XX
3	1				XX.XX.XX.XX.XX
4	1	670599	back cover with cushion 2 cm		XX.XX.XX.XX.XX
o.B.	2		counter sunk screw with hexogon socket DIN 7991 M5x16		01.65.07.92.05.00
o.B.	2		rosette ø13/6x2, black		04.14.70.60.02.00

Curved back plate


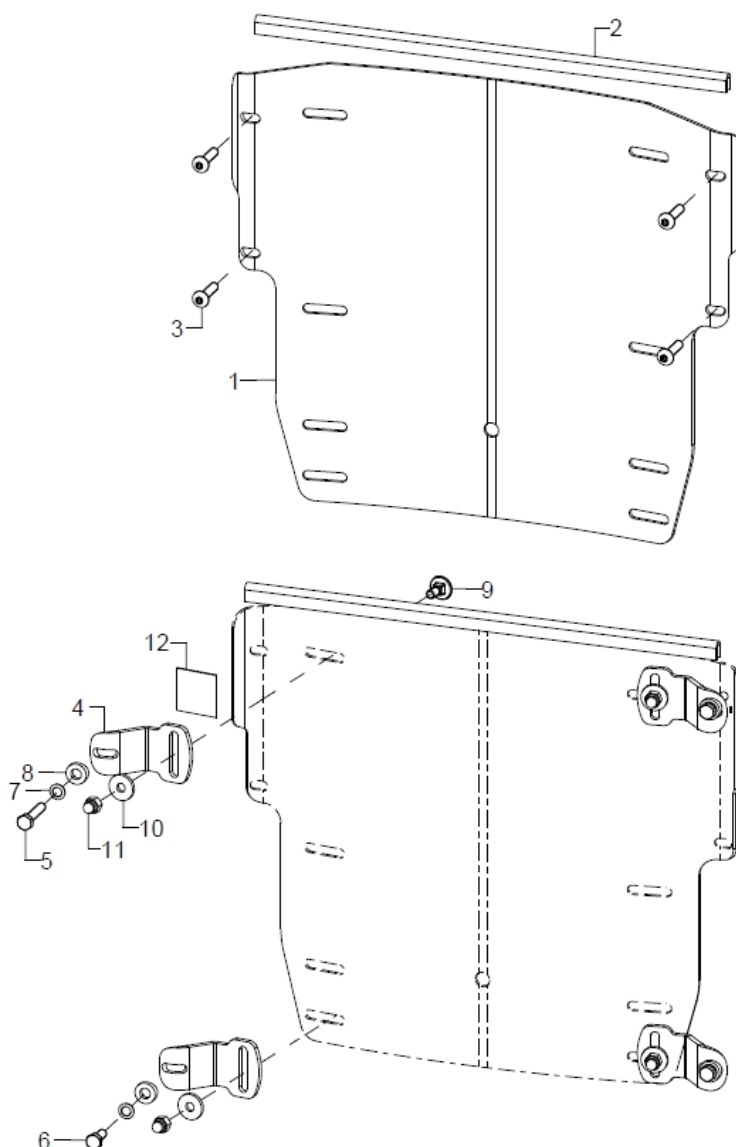


pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1	1	6930XX			69.30.XX.00.00.00
o.B.	1	6931XX			69.31.XX.00.00.00
o.B.	1	6932XX			69.32.XX.00.00.00
o.B.	1	6933XX			69.33.XX.00.00.00
o.B.	1	6934XX			69.34.XX.00.00.00
o.B.	1	6935XX			69.35.XX.00.00.00
o.B.	1	6950XX			69.50.XX.00.00.00
o.B.	1	6951XX			69.51.XX.00.00.00
o.B.	1	6952XX			69.52.XX.00.00.00
o.B.	1	6953XX			69.53.XX.00.00.00
o.B.	1	6954XX			69.54.XX.00.00.00
o.B.	1	6955XX			69.55.XX.00.00.00
2	1	450030	edge protection-steel clamp band, 1-2 mm black		00.45.07.10.99.10
3	4				01.42.07.95.65.00
4+5+7-12	2	692598	mounting set curved back plate		XXXXXXXXXXXX
4+6-11	2				




4	1	692510	screwed on holding plate	69.25.05.01.00.69
5	1		hexagon screw DIN 933 - M5x25	01.04.07.96.02.40
6	1		hexagon screw DIN 933 - M5x12	01.04.07.89.02.40
7	1		lock washer S5	02.99.10.50.01.10
8	1			40.04.16.01.00.10
9	1		round-head screw DIN 6003 M5x12	01.11.07.89.00.40
10	1		washer DIN 7349 - 5,3	02.77.05.22.00.20
11	1		hexagon cap nut low form with clamp DIN 986 - M5	02.33.00.60.21.20
12	2		Velcro strap 25 mm	00.42.61.02.50.11

Übersicht Muldenrücken

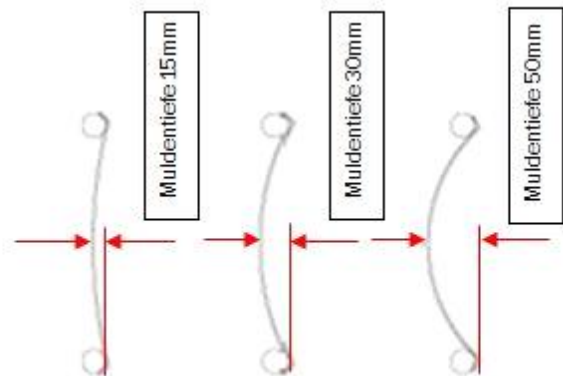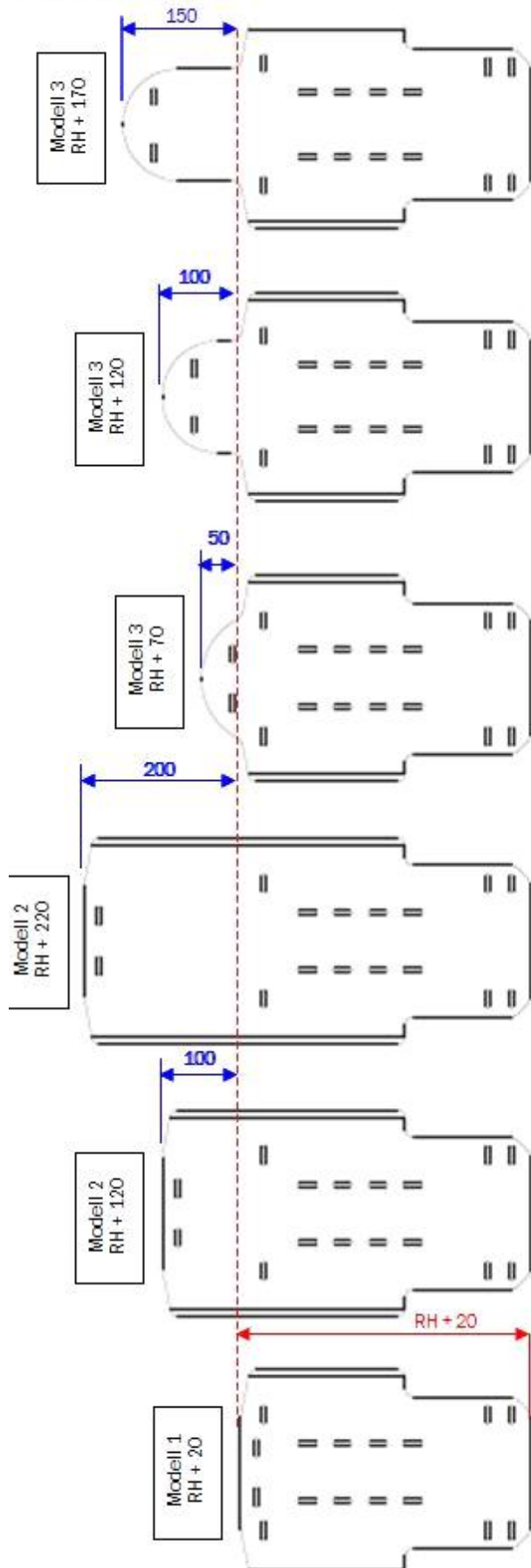

Back elongation

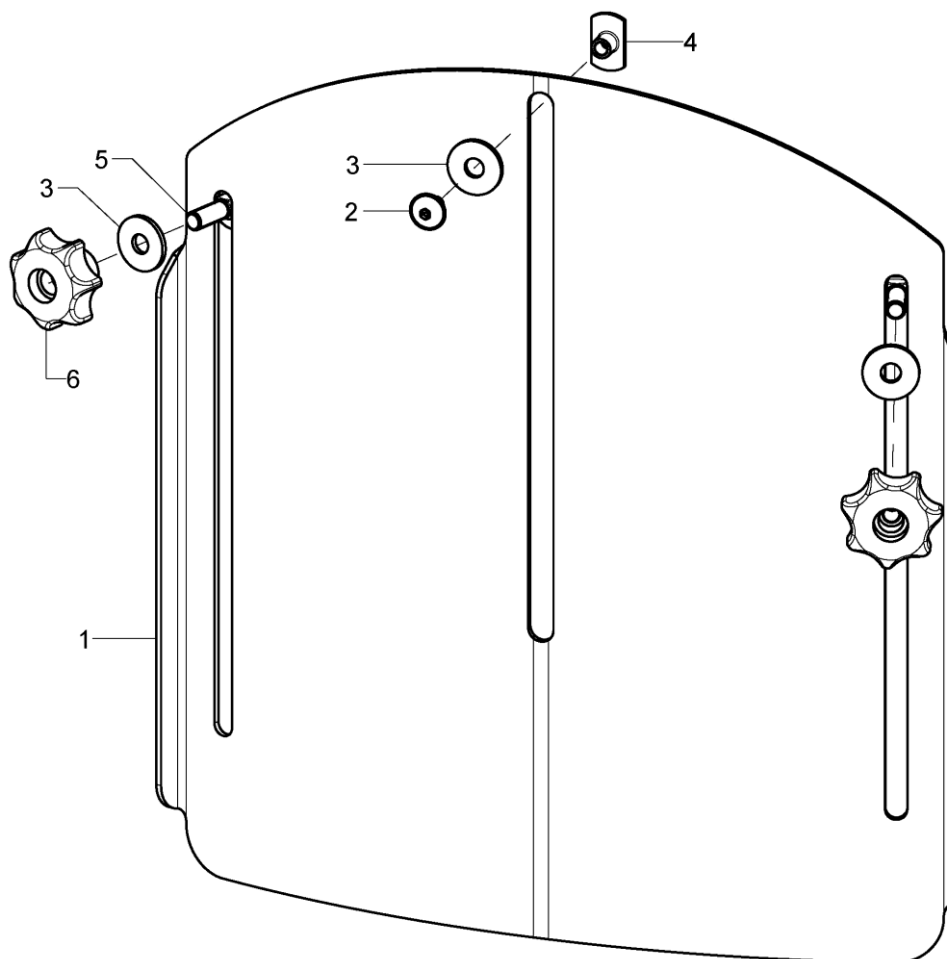

pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1	1	686091			68.60.01.00.00.00
o.B.	1	686092			68.60.02.00.00.00
2-6	1	68607299	mounting set back extension		XXXXXXXXXXXX
2	1		lens head screw with hexagon socket M5x8		01.43.07.87.62.60
3	3		washer DIN 9021- 6,4- PA		02.79.06.41.60.70
4	1		weld nut C- M5x7,5		02.49.33.03.00.10
5	2		round-head screw DIN 6003 M6x20		01.11.08.54.00.40
6	2		7-star knob ø32 - M6 internal thread continuous, black		04.05.37.64.20.30


Cushion for curved back plate

pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
o.B.	1	8517	back cushion 3cm with zipper		XXXXXXXXXXXXX
o.B.	1	8517XX	cover for back cushion		XXXXXXXXXXXXX
o.B.	1	891104	anatomical back (3cm) incl. changeable cover		
o.B.	1	891106	anatomical back (5cm) incl. changeable cover		
o.B.	1	8911XX	cover for anatomical back		XXXXXXXXXXXXX
o.B.	1	68607201	head rest pad incl. cover for back extension		

Stoffmuster	XX
Betex schwarz	20
Betex blau	21
Betex rot	22
Betex grau	23
Betex grün	24
Betex gelb	25
Betex orange	27


Alle Stoffmuster können unter www.sorgrollstuhltechnik.de angesehen werden.

Truss pads Stealth from BH 35cm

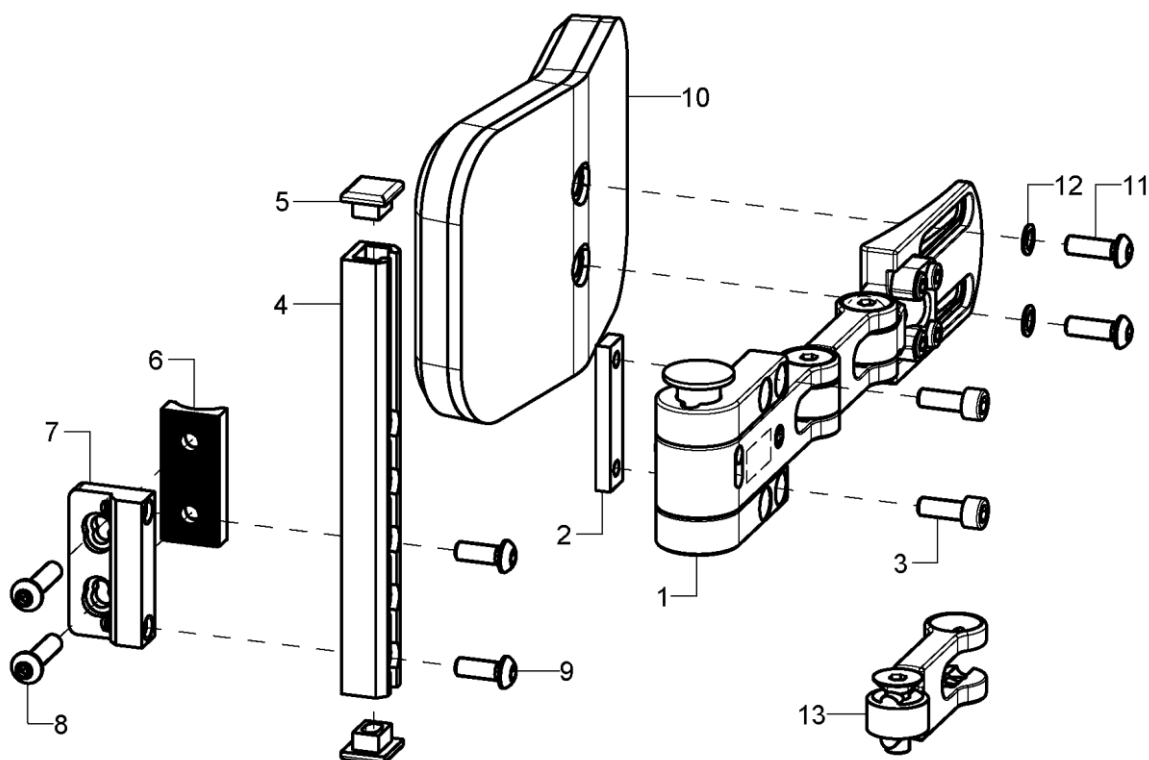


pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1	1	68590202			68.59.02.00.02.00
o.B.	1	68590201			68.59.02.00.01.00
2-9	1	68590111			68.59.01.11.90.00
2-5+9	1	68590112			XX.XX.XX.XX.XX.XX
2	1				68.59.02.12.00.00
3	2		cylinder screw DIN 912 M6x16		01.22.08.52.62.40
4	1				68.59.01.20.00.00
5	2				35.01.02.05.00.10
6	1				68.57.01.05.00.00
7	1				68.57.01.03.00.00
8	2		lens head screw with hexagon socket ISO 7380 - M6x25		01.42.08.56.65.00
9	2		lens head screw with hexagon socket ISO 7380 - M6x12		01.42.08.49.65.00
o.B.	2		counter sunk screw with hexagon socket DIN 7991 M5x16		01.65.07.92.05.00
10-12	1	68590204			68.59.01.23.00.00
o.B.	1	68590205			68.59.01.24.00.00
11	2		lens head screw with hexagon socket ISO 7380 - M6x16		01.42.08.52.65.00
12	2		lock washer S6		02.99.10.60.01.10
13	1	68590203			68.59.02.15.00.00

Head rest

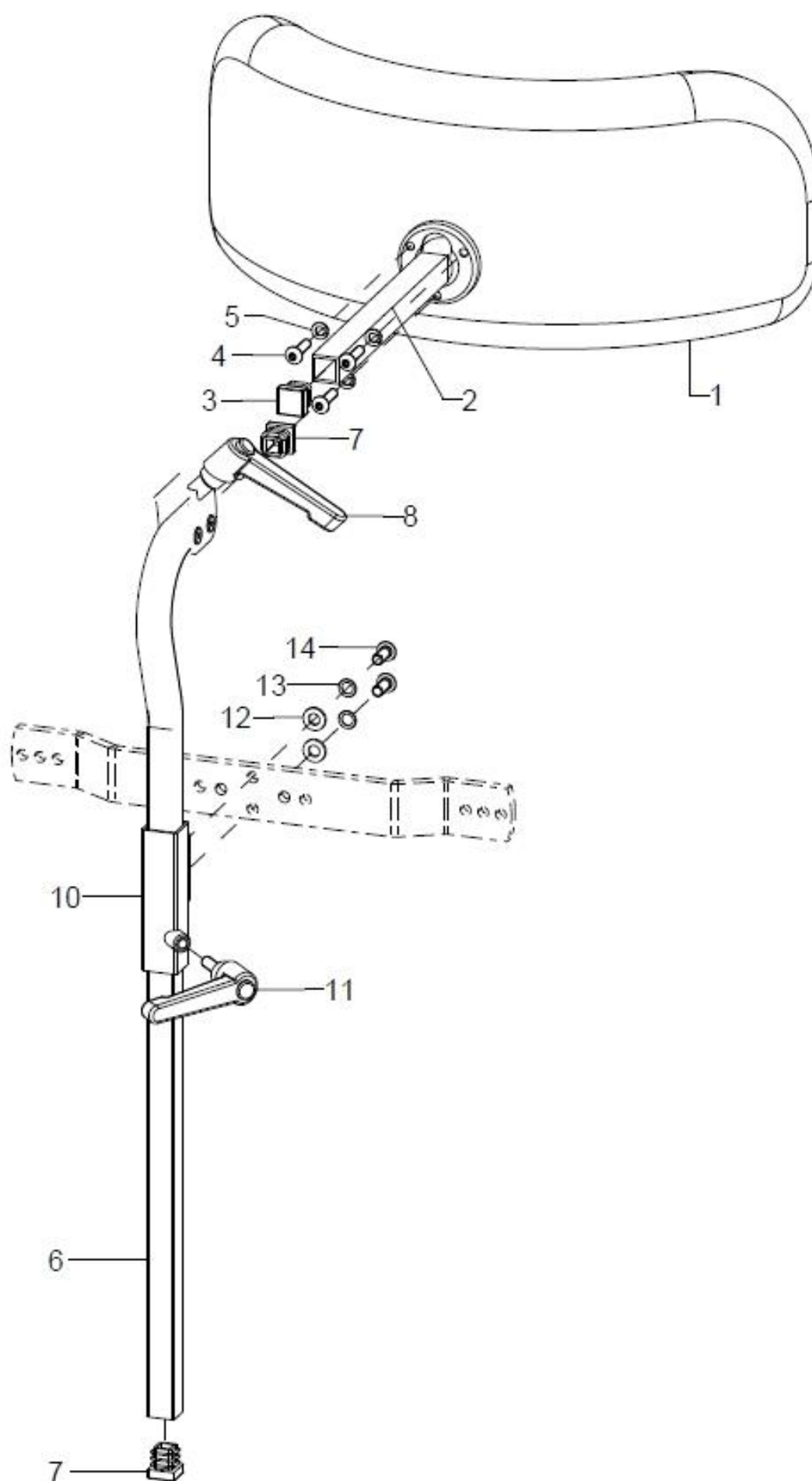


pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-14	1	686021			
altern.	1	686022			
altern.	1	686023			
altern.	1	686024			
altern.	1	686025			
altern.	1	686026			
1-5	1	686031	head rest cushion model 1, 213x145 mm shell form complete		68.60.15.99.00.10
altern.	1	686032	head rest cushion model 2, 220x189 mm shell form complete		68.60.18.99.00.10
altern.	1	686033	head rest model 3, 225x100 curved complete		68.60.16.99.00.10
altern.	1	686034	head rest model 4, 300x150 curved complete		68.60.19.99.00.10
altern.	1	686035	head rest model 5, 230x120 , 4-curved complete		68.60.20.99.00.10
altern.	1	686036	head rest model 5, 20x140 , 4-curved complete		68.60.21.99.00.10
2	1				
3	1		plugs for square tubes, black 15x15		04.63.11.15.00.10
4	3		lens head screw with hexagon socket ISO 7380 - M5x20		01.42.07.94.65.00
5	3		spring ring DIN 127 - A 5		02.85.30.50.05.10
6-8	1	68600801			68.60.08.99.01.00
o.B.	1	68600802			68.60.08.99.02.00
7	2		plugs for square tubes, black 15x15		04.63.11.15.00.10
8	1	403110	clamp lever adjustable M6x20, black		04.13.10.60.20.50
10-14	1	68600804			68.60.08.04.00.00
11	1	403110	clamp lever adjustable M6x20, black		04.13.10.60.20.50
12	2		washer DIN 125 - A 6,4		02.61.06.41.41.20
13	2		lock washer S6		02.99.10.60.01.10
14	2				

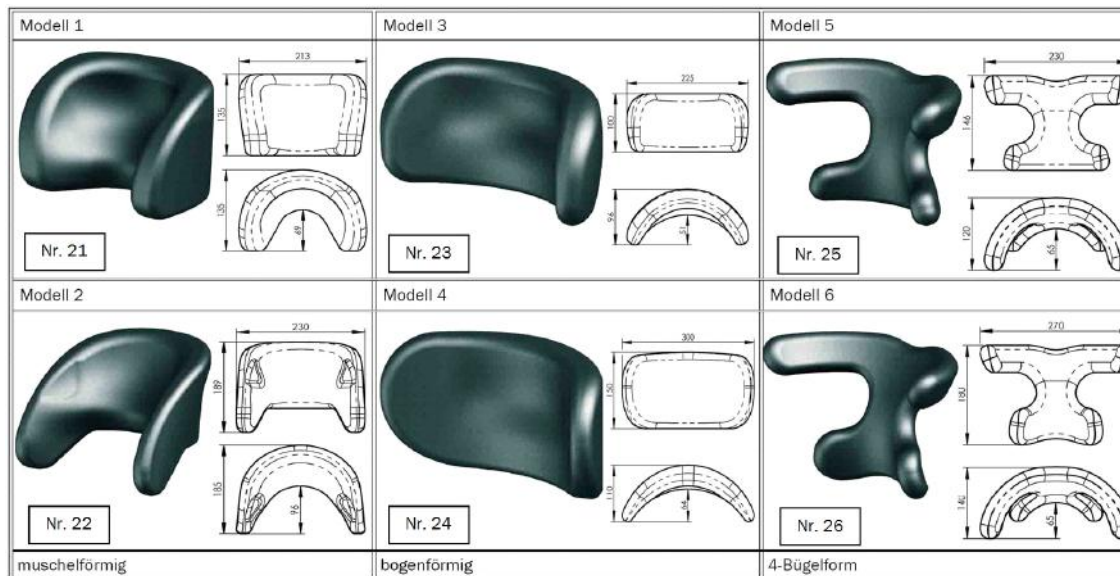


Side guards standard (from 10.06.2013)


Verschraubung hinten bei Rücken Standard

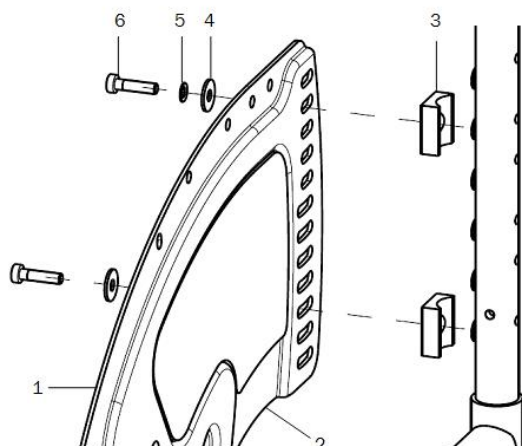


Verschraubung hinten bei Rücken winkelverstellbar

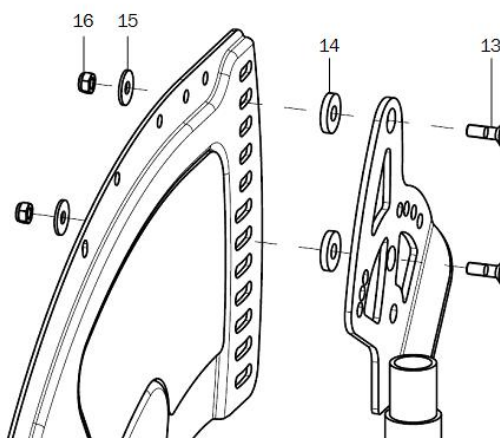


Verschraubung vorne mit Sitzplattenträger, bzw. Bspannungsträger bei Sitztiefe 22 - 32

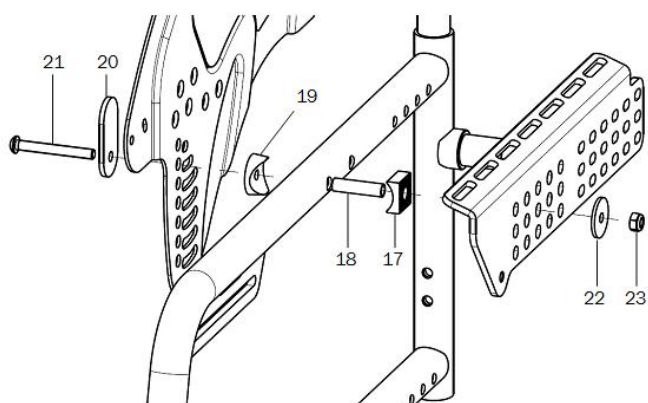

Verschraubung vorne ohne Sitzträger, bei Sitztiefe 34 - 40

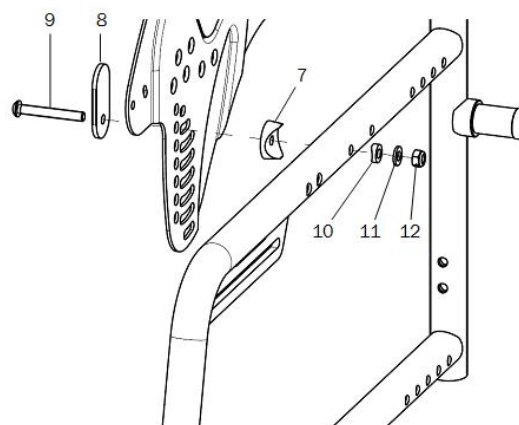

Seitenteilverlängerung

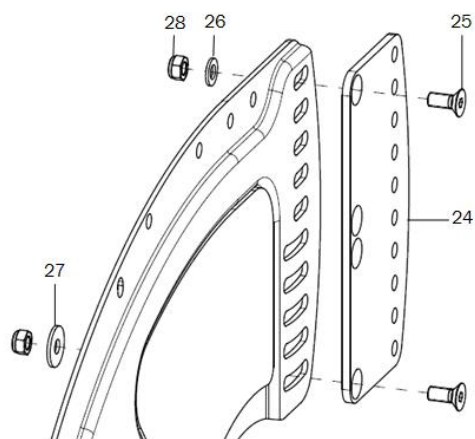



Spare parts list Vector


Side part

Page 66 of 127



pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1	1	45252202	side guard 20" /22" right		45.25.01.01.02.74
o.B.	1	45252201	side guard 20" / 22" left		45.25.01.01.01.74
altern.	1	45252402	side guard 24" right		45.25.01.02.02.74
o.B.	1	45252401	side guard 24" left		45.25.01.02.01.74
altern.	1	45252602			45.25.05.11.02.75
o.B.	1	45252601			45.25.05.11.01.75
2	1	452522	insulation pad 20"/24"		45.25.01.20.00.74
o.B.	1	452524	insulation pad 24"		45.25.01.24.00.74
3-6+17-23	2				
13-23	2				
3-12	2				
7-16	2				
3	2				45.17.01.06.00.69
4	2		washer DIN 9021 - M5		02.79.05.31.20.20
5	2		lock washer S5		02.99.10.50.01.10
6	2		cylinder head screw low head DIN 912 - M5x20		01.23.07.94.02.40
7	1		saddle washer black ø 24/6		04.63.39.24.06.30
8	1				45.25.01.11.00.74
9	1		lens head screw with hexagon socket ISO 7380 - M6x45		01.42.08.61.65.00
10	1				40.04.16.01.00.10
11	1		washer DIN 125 - A 6,4		02.61.06.41.41.20
12	1		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20
13	2		counter sunk screw with hexagon socket DIN 7991 M5x16		01.65.07.92.05.00
14	2		washer DIN 7349 - 6,4		02.77.06.53.00.20
15	2		washer DIN 9021 - M5		02.79.05.31.20.20
16	1		hexagon nut self-locking low form DIN 985 - M5		02.32.00.50.01.20
17	1				76.01.03.01.00.10
18	1				19.08.01.01.00.69
19	1		saddle washer black ø 24/6		04.63.39.24.06.30
20	1				45.25.01.11.00.74
21	1		lens head screw with hexagon socket ISO 7380 - M6x50		01.42.08.62.65.00
22	1				40.24.01.04.00.54
23	1		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20
24-28		452502	side guard extension incl. mounting set		45.25.02.01.00.74
25	2		counter sunk screw with hexagon socket DIN 7991 M5x12		01.65.07.89.05.00
26	1		washer DIN 125 - A 5,3		02.61.05.31.01.20
27	1		washer DIN 9021 - M5		02.79.05.31.20.20
28	2		hexagon nut self-locking low form DIN 985 - M5		02.32.00.50.01.20

Side guards standard (until 09.06.2013)


Verschraubung hinten bei Rücken Standard

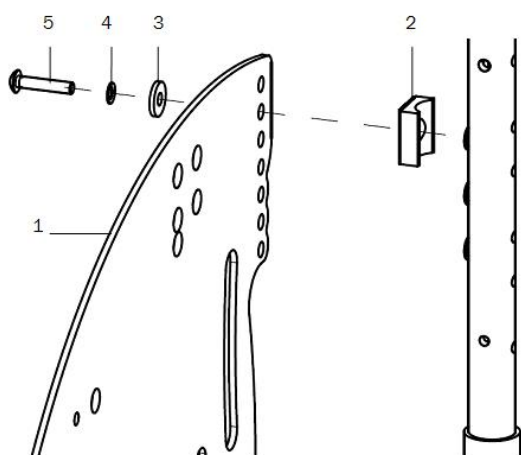

Verschraubung hinten bei Rücken winkelverstellbar

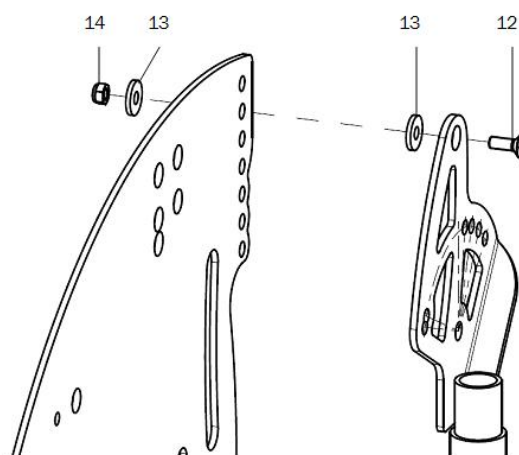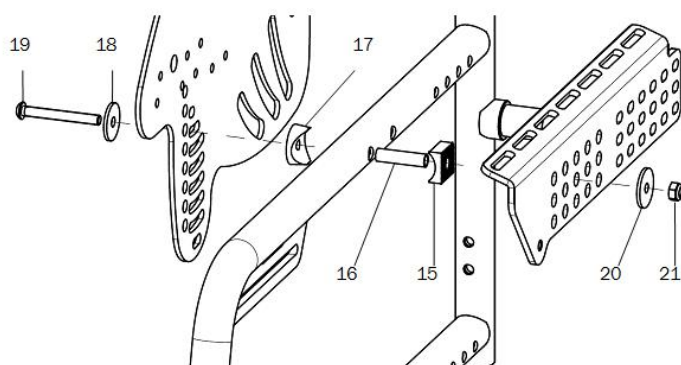
Verschraubung vorne mit Sitzplattenträger,
bzw. Bspannungsträger bei Sitztiefe 22 - 32


Verschraubung vorne ohne Sitzträger, bei Sitztiefe 34 - 40

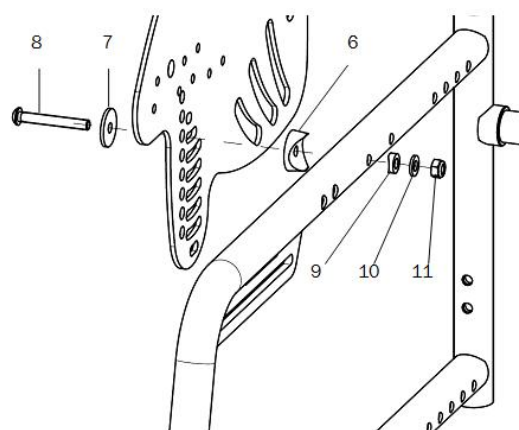




Spare parts list Vector

Side part

Page 68 of 127


pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1	1	45170102	side guard aluminum 22", right		45.17.01.02.02.69
o.B.	1	45170101	side guard aluminum 22", left		45.17.01.02.01.69
altern.	1	45170302	side guard aluminum 24", right		45.17.01.04.02.69
o.B.	1	45170301	side guard aluminum 24", left		45.17.01.04.01.69
2-5+15-21	2	45170190	mounting set side guard aluminum, SD 22-32, standard back		XXXXXXXXXXXXX
12-21	2	45170195	mounting set side guard aluminum, SD 22-32, angle adjustable back		XXXXXXXXXXXXX
2-11	2	45170191	mounting set side guard aluminum, SD 34-40, standard back		XXXXXXXXXXXXX
6-14	2	45170196	mounting set side guard aluminum, SD 34-40, angle adjustable back		XXXXXXXXXXXXX
2	1				45.17.01.06.00.69
3	1		washer DIN 7349 - 5,3		02.77.05.22.00.20
4	1		lock washer S5		02.99.10.50.01.10
5	1		lens head screw with hexagon socket ISO 7380 - M5x25		01.42.07.96.65.00
6	1		saddle washer black ø 24/6		04.63.39.24.06.30
7	1				
8	1		lens head screw with hexagon socket ISO 7380 - M6x45		01.42.08.61.65.00
9	1				40.04.16.01.00.10
10	1		washer DIN 125 - A 6,4		02.61.06.41.41.20
11	1		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20
12	1		counter sunk screw with hexagon socket DIN 7991 M5x14		01.65.07.90.05.00
13	2		washer DIN 7349 - 5,3		02.77.05.22.00.20
14	1		hexagon nut self-locking low form DIN 985 - M5		02.32.00.50.01.20
15	1				76.01.03.01.00.10
16	1				19.08.01.01.00.69
17	1		saddle washer black ø 24/6		04.63.39.24.06.30
18	1				
19	1		lens head screw with hexagon socket ISO 7380 - M6x50		01.42.08.62.65.00
20	1				40.24.01.04.00.54
21	1		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20

Arm rest for side guards standard (from 10.06.2013)

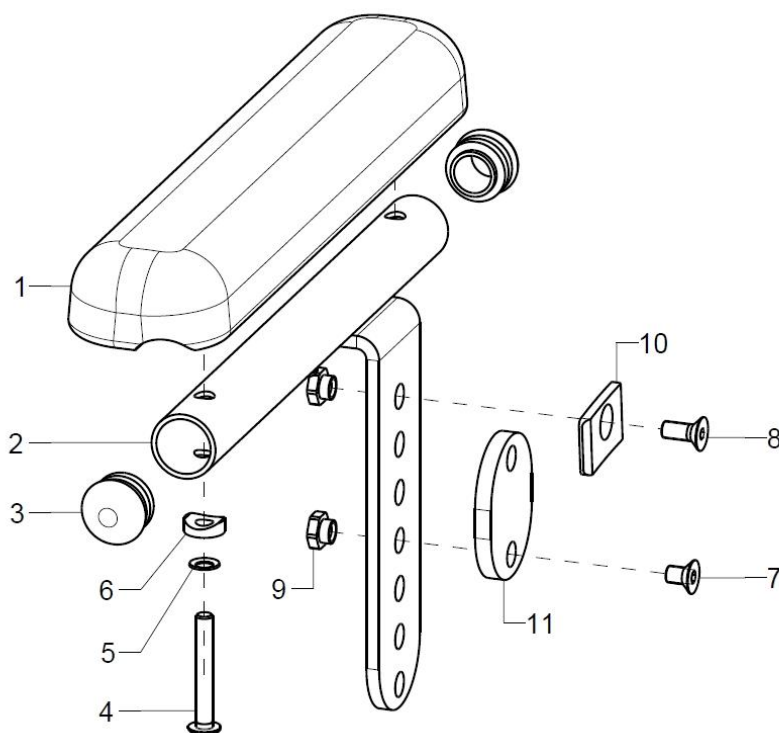


pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-10	1P	452600	arm pad for standard side guard, complete		XXXXXXXXXXXX
1-9+11	1P	452601			
1	2	450007	arm rest, straight black 240x50x26, 2xM5x164		04.65.01.24.00.10
2-3	1P	452603	holder for arm pad incl 3		45.26.05.00.00.74
3	2		plugs for tube ø20x1-2, black		04.63.10.20.00.10
4-10	2	452699	mounting set arm pad		XXXXXXXXXXXX
4-9+11	2	452698			
4	2		lens head screw with hexagon socket ISO 7380 - M5x35		01.42.07.99.65.00
5	2		lock washer S5		02.99.10.50.01.10
6	2				40.04.16.01.00.10
7	1		counter sunk screw with hexagon socket DIN 7991 M5x8		01.65.07.87.05.00
o.B.	1		counter sunk screw with hexagon socket DIN 7991 M5x14		01.65.07.90.05.00
8	1		counter sunk screw with hexagon socket DIN 7991 M5x12		01.65.07.89.05.00
o.B.	1		counter sunk screw with hexagon socket DIN 7991 M5x14		01.65.07.90.05.00
9	2				45.25.01.13.00.74
10	1				
11	1				45.26.05.01.00.74


Arm rest for side guards standard (until 09.06.2013)

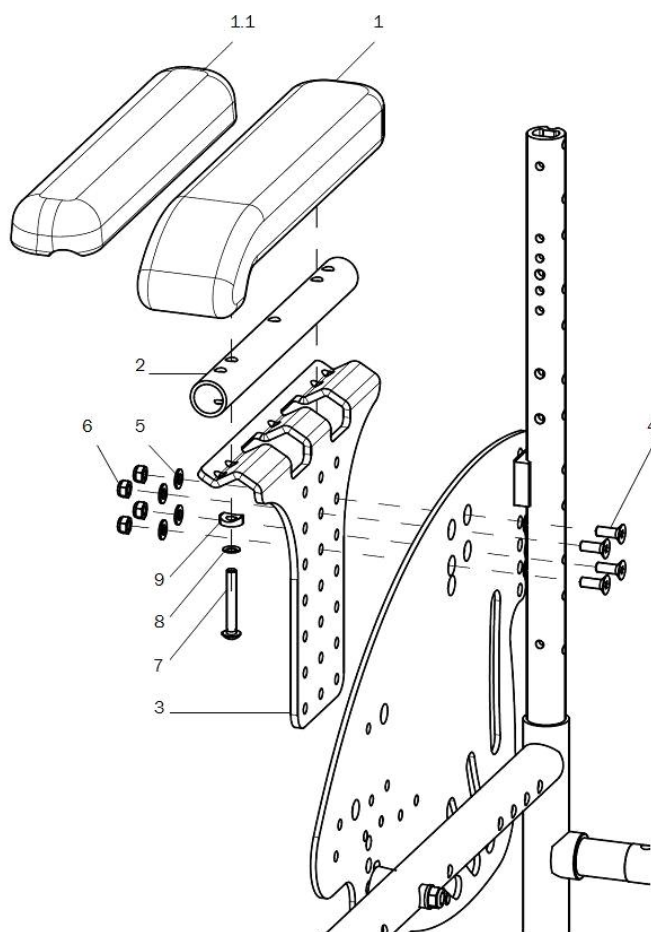

pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-9	1	452210	arm pad for side guard aluminum complete		XXXXXXXXXXXX
1	2	450008	arm rest, bent black 255x52x33, 2xM5x130		04.65.01.25.00.10
1.1	2	450007	arm rest, straight black 240x50x26, 2xM5x164		04.65.01.24.00.10
2	2	452215	arm rest rod 130/164		45.22.02.07.00.20
3	2	452207	holder for arm rest 180x36x201 height adjustable		45.22.02.01.00.20
4-9	2	450099	mounting set arm rest		XXXXXXXXXXXX
4	4		counter sunk screw with hexagon socket DIN 7991 M5x14		01.65.07.90.05.00
5	4		washer DIN 125 - A 5,3		02.61.05.31.01.20
6	4		hexagon nut self-locking low form DIN 985 - M5		02.32.00.50.01.20
7	2		lens head screw with hexagon socket ISO 7380 - M5x35		01.42.07.99.65.00
8	2		lock washer S5		02.99.10.50.01.10
9	2				40.04.16.01.00.10

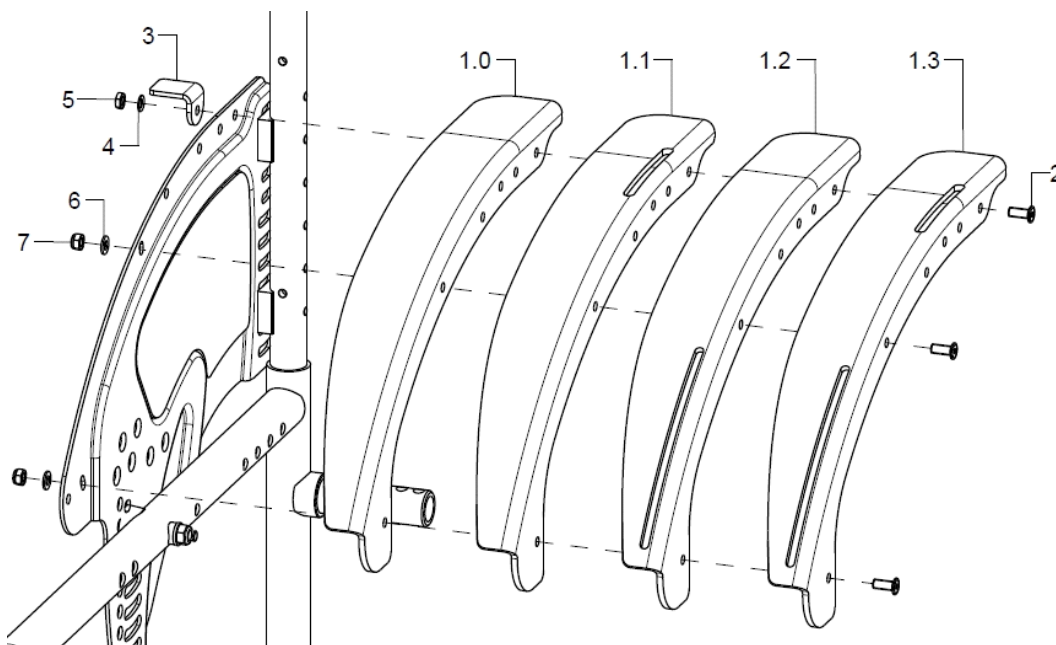
Clothes guard side guard (from 10.06.2013)


pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1	1	50240102	skirtguard 22" standard, right		50.24.01.01.02.74
o. B.	1	50240101	skirtguard 22" standard, left		50.24.01.01.01.74
alt.	1	50240202	skirtguard 24" standard, right		50.24.01.02.02.74
o. B.	1	50240201	skirtguard 24" standard, left		50.24.01.02.01.74
1.1	1	50240302	skirtguard 22" for arm rest, right		50.24.01.03.02.74
o. B.	1	50240301	skirtguard 22" for arm rest, left		50.24.01.03.01.74
alt.	1	50240402	skirtguard 24" for arm rest, right		50.24.01.04.02.74
o. B.	1	50240401	skirtguard 24" for arm rest, left		50.24.01.04.01.74
1.2	1	50240502	skirtguard 22" for cable brake, right		50.24.01.05.02.74
o. B.	1	50240501	skirtguard 22" for cable brake, left		50.24.01.05.01.74
alt.	1	50240602	skirtguard 24" cable brake, right		50.24.01.06.02.74
o. B.	1	50240601	skirtguard 24" for cable brake, left		50.24.01.06.01.74
1.3	1	50240702	skirtguard 22" cable brake/arm rest, right		50.24.01.07.02.74
o. B.	1	50240701	skirtguard 22" cable brake/arm rest, left		50.24.01.07.01.74
alt.	1	50240802	skirtguard 24" cable brake/arm rest, right		50.24.01.08.02.74
o. B.	1	50240801	skirtguard 24" cable brake/arm rest, left		50.24.01.08.01.74
2-7	2	502499	mounting set skirt guard		XXXXXXXXXXXX
2	3				01.66.07.89.95.00
3	1	50240115			50.24.01.15.00.74
4	1		lock washer S5		02.99.10.50.01.10
5	1		low hexagon nut DIN 439 - M5 - 05 vz		02.03.00.50.07.20
6	2		washer DIN 125 - A 5,3		02.61.05.31.01.20
7	2		hexagon nut self-locking low form DIN 985 - M5		02.32.00.50.01.20

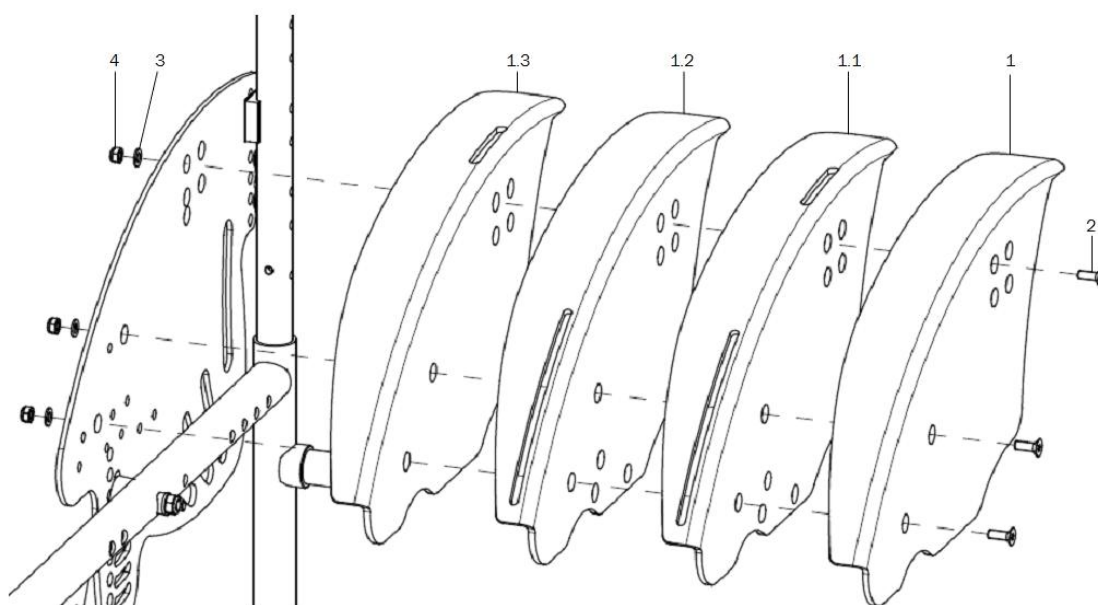Clothes guard side guard (until 09.06.2013)


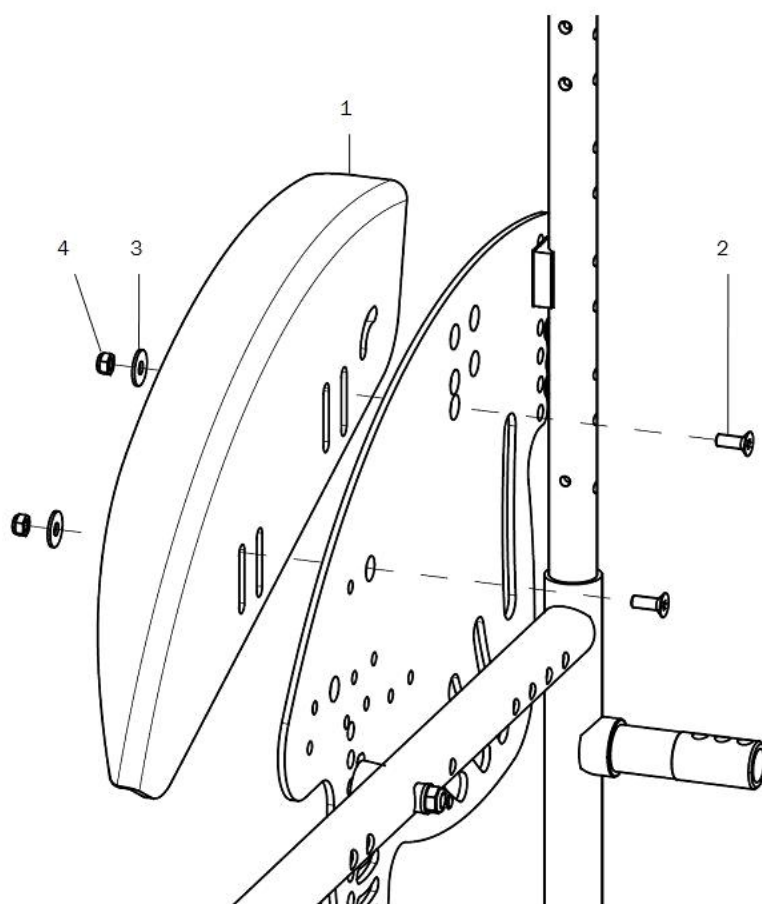
pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1	1	50210102	skirtguard 22" standard, right		50.21.01.01.02.69
o. B.	1	50210101	skirtguard 22" standard, left		50.21.01.01.01.69
alt.	1	50210202	skirtguard 24" standard, right		50.21.01.02.02.69
o. B.	1	50210201	skirtguard 24" standard, left		50.21.01.02.01.69
1.1	1	50210902	skirtguard 22" cable brake/arm rest, right		50.21.01.09.02.69
o.B.	1	50210901	skirtguard 22" cable brake/arm rest, left		50.21.01.09.01.69
alt.	1	50211002	skirtguard 24" cable brake/arm rest, right		50.21.01.10.02.69
o. B.	1	50211001	skirtguard 24" cable brake/arm rest, left		50.21.01.10.01.69
1.2	1	50210502	skirtguard 22" for cable brake, right		50.21.01.07.02.69
o.B.	1	50210501	skirtguard 22" for cable brake, left		50.21.01.07.01.69
alt.	1	50210602	skirtguard 24" cable brake, right		50.21.01.08.02.69
o. B.	1	50210601	skirtguard 24" for cable brake, left		50.21.01.08.01.69
1.3	1	50210302	skirtguard 22" for arm rest, right		50.21.01.03.02.69
o.B.	1	50210301	skirtguard 22" for arm rest, left		50.21.01.03.01.69
alt.	1	50210402	skirtguard 24" for arm rest, right		50.21.01.04.02.69
o. B.	1	50210401	skirtguard 24" for arm rest, left		50.21.01.04.01.69
2-4	2	50210099	mounting set skirtguard		XXXXXXXXXXXX
2	1		counter sunk screw with hexagon socket DIN 7991 M5x14		01.65.07.90.05.00
3	1		washer DIN 125 - A 5,3		02.61.05.31.01.20
4	1		hexagon nut self-locking low form DIN 985 - M5		02.32.00.50.01.20

Arm rest for clothes guard (until 10.06.2013)

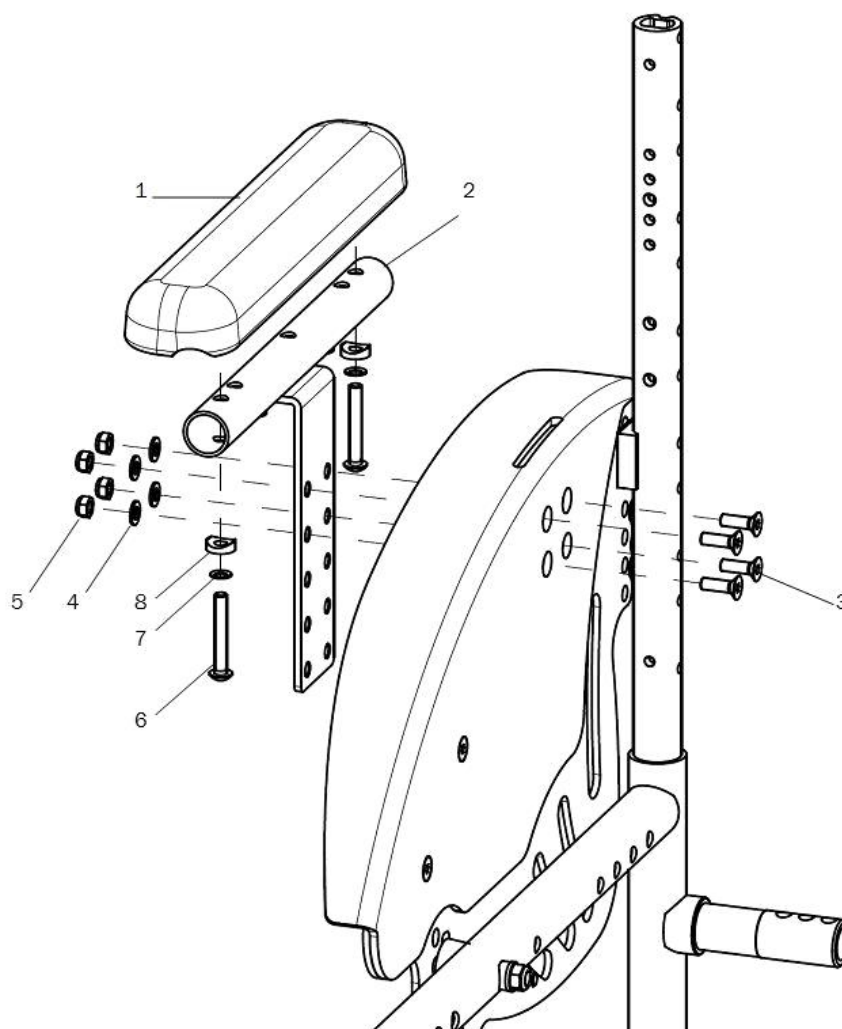


pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-8	2	460005	arm pad for skirt guard, complete		XXXXXXXXXXXX
1	2	460007	arm pad straight black 240x50x26, 2xM5x164		
2	2	460204	holder-arm rest		46.00.04.01.00.10
3-8	2	460299	mounting set arm pad		XXXXXXXXXXXX
3	4				
4	4				
5	4				
6	2				
7	2				
8	2				

Wheel cover (until 10.06.2013)



pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1	1	52050002	wheel cover 20" right		52.02.02.20.02.10
o.B.	1	52050001	wheel cover 20" left		52.02.02.20.01.10
alt.	1	52050102	wheel cover 22" right		52.02.02.22.02.10
o.B.	1	52050101	wheel cover 22", left		52.02.02.22.01.10
alt.	1	52050202	wheel cover 24" right		52.02.02.24.02.10
o.B.	1	52050201	wheel cover 24" left		52.02.02.24.01.10
2-4	2				
2	1		counter sunk screw with hexagon socket DIN 7991 M5x14		01.65.07.90.05.00
3	1		washer DIN 9021 - M5		02.79.05.31.20.20
4	1		hexagon nut self-locking low form DIN 985 - M5		02.32.00.50.01.20

Desk side guard

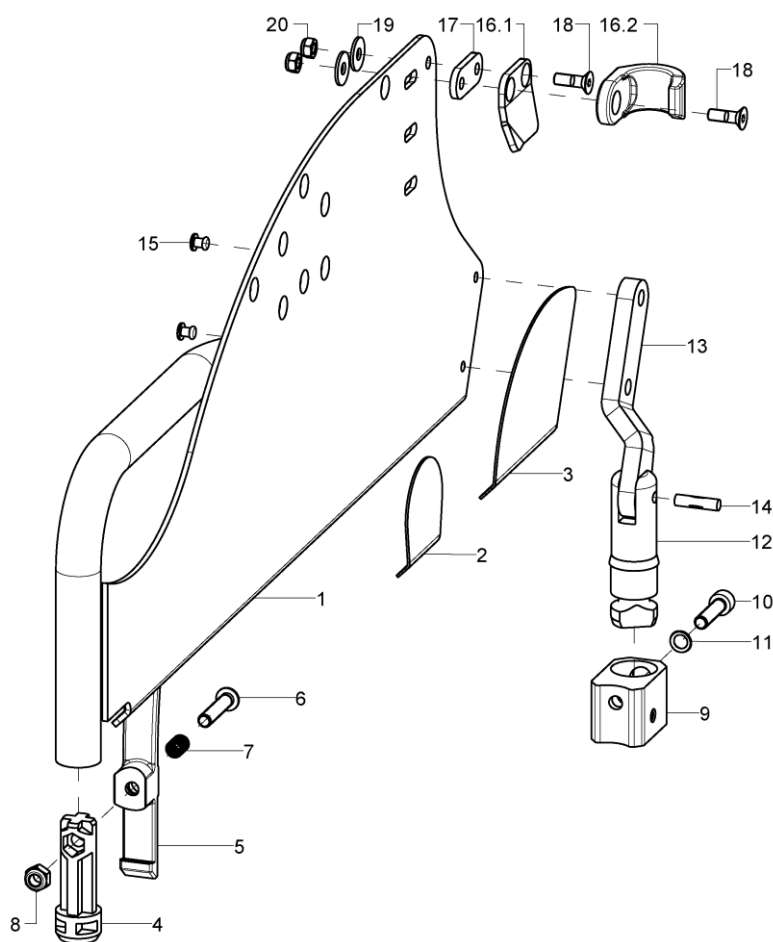

pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1	1	46100202			46.10.02.01.02.75
o.B.	1	46100201			46.10.02.01.01.75
altern.	1	46100302			46.10.03.01.02.75
o.B.	1	46100301			46.10.03.01.01.75
2	2	46100121			46.10.01.21.00.75
3	2	46100120			46.10.01.20.00.75
o.B.	1	45170202			45.17.01.05.02.69
o.B.	1	45170201			45.17.01.05.01.69
4	2	46100113			46.10.01.13.00.75
5-8	2	46100114			46.10.01.14.00.75
6	1		lens head screw with hexagon socket ISO 7380 - M6x30		01.42.08.58.65.00
7	1		pressure spring DIN 2098 0,7x7,65/9,05x21		04.21.00.80.21.00
8	1		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20
9-11	2	46100117			46.10.99.17.00.75
10	1		cylinder screw DIN 912 M6x25		01.22.08.56.02.40
11	1		lock washer S6		02.99.10.60.01.10
12-15	1	46101802			46.10.01.18.02.75
o.B.	1	46101801			46.10.01.18.01.75
13	1				46.10.01.15.02.75
o.B.	1				46.10.01.15.01.75
14	1				03.36.05.20.00.01
15	2		blind rivet F5x10 A2/A2		03.64.00.50.10.30




Spare parts list Vector

Side part

Page 76 of 127


16.1-20	1	46101602	46.10.01.16.02.75
o.B.	1	46101601	46.10.01.16.01.75
16.2+18-20	2	45260110	45.26.01.10.00.75
17	2		46.10.01.19.00.75
18	2(1)	counter sunk screw with hexogon socket DIN 7991 M5x16	01.65.07.92.05.00
19	2(1)	washer DIN 9021 - M5	02.79.05.31.20.20
20	2(1)	hexagon nut self-locking low form DIN 985 - M5	02.32.00.50.01.20

Clothes guard for desk side guards

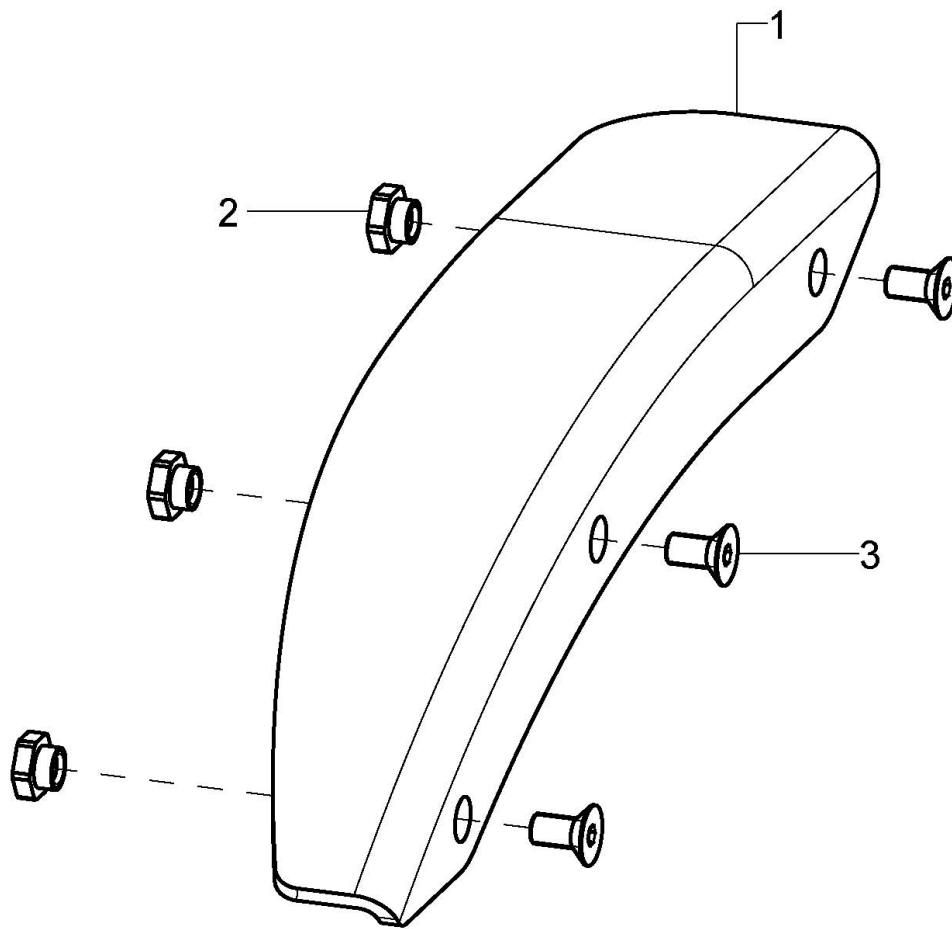


pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-3	1	46100102			46.10.01.24.02.75
o.B.	1	46100101			46.10.01.24.01.75
2	3				45.25.01.13.00.74
3	3		counter sunk screw with hexogon socket DIN 7991 M5x10		01.65.07.88.05.00

Arm rest for desk side guards

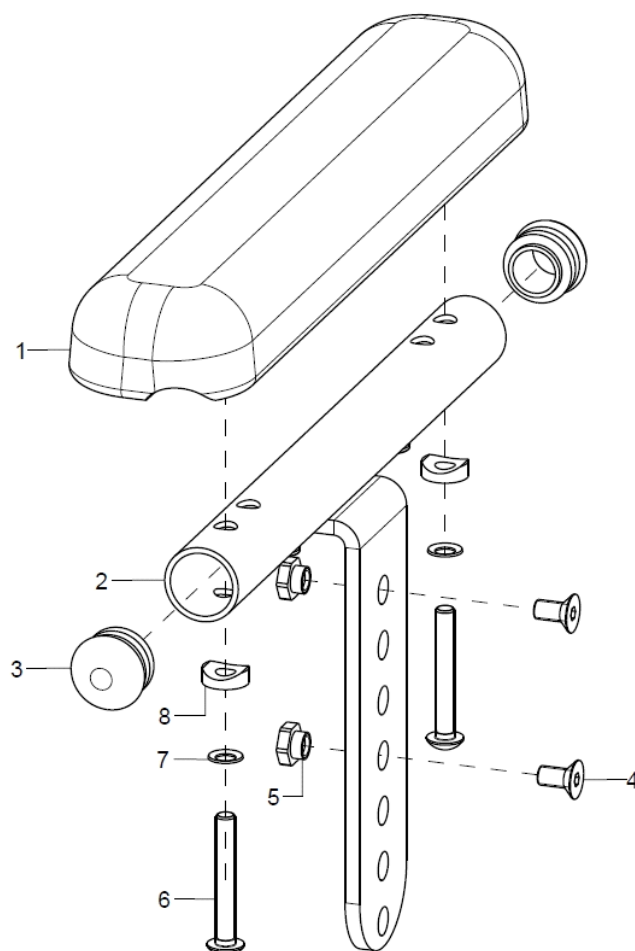

pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-8	1	461005			
1	1	450007	arm rest, straight black 240x50x26, 2xM5x164		04.65.01.24.00.10
2-5	1	461002			45.25.03.00.00.74
3	2	620002	plug for rod ø20x1-2, black		04.63.10.20.00.10
4	2		counter sunk screw with hexagon socket DIN 7991 M5x8		01.65.07.87.05.00
5	2				45.25.01.13.00.74
6-8	1	452697			
6	2		lens head screw with hexagon socket ISO 7380 - M5x35		01.42.07.99.65.00
7	2		lock washer S5		02.99.10.50.01.10
8	2				40.04.16.01.00.10

Long side guard

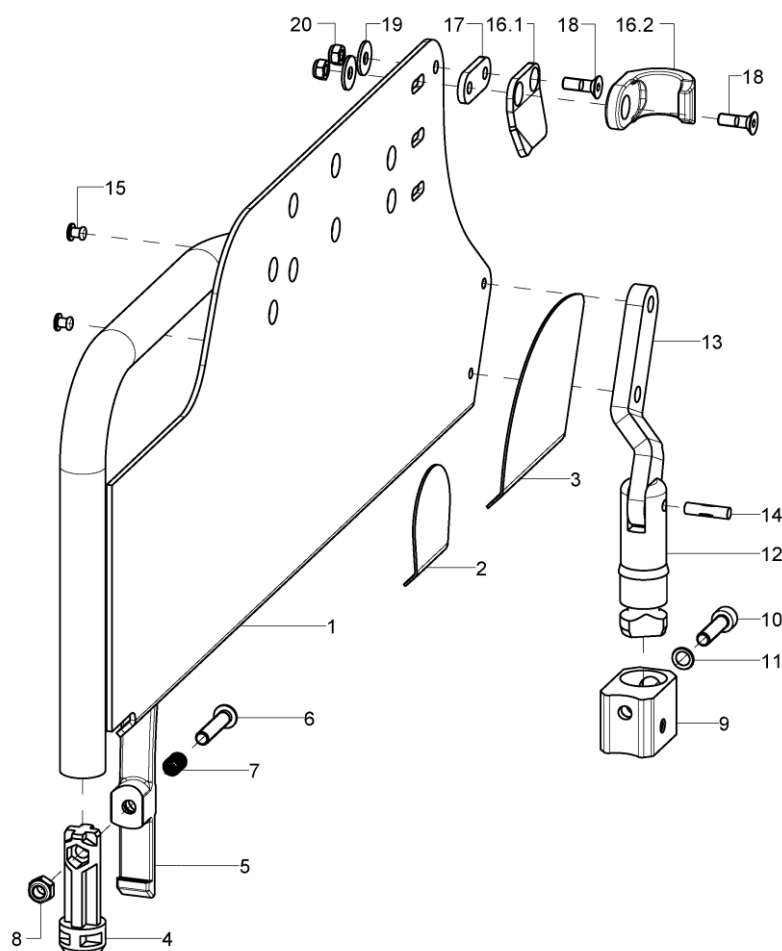


pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1	1	46110202			46.11.XX.01.02.75
o.B.	1	46110201			46.11.XX.01.01.75
altern.	1	46110302			
o.B.	1	46110301			
2	2	46100121			46.10.01.21.00.75
3	2	46100120			46.10.01.20.00.75
o.B.	1	45170202			45.17.01.05.02.69
o.B.	1	45170201			45.17.01.05.01.69
4	2	46100113			46.10.01.13.00.75
5-8	2	46100114			46.10.01.14.00.75
6	1		lens head screw with hexagon socket ISO 7380 - M6x30		01.42.08.58.65.00
7	1		pressure spring DIN 2098 0,7x7,65/9,05x21		04.21.00.80.21.00
8	1		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20
9-11	2	46100117			46.10.99.17.00.75
10	1		cylinder screw DIN 912 M6x25		01.22.08.56.02.40
11	1		lock washer S6		02.99.10.60.01.10
12-15	1	46101802			46.10.01.18.02.75
o.B.	1	46101801			46.10.01.18.01.75
13	1				46.10.01.15.02.75
o.B.	1				46.10.01.15.01.75
14	1				03.36.05.20.00.01
15	2		blind rivet F5x10 A2/A2		03.64.00.50.10.30



16.1-20	1	46101602	46.10.01.16.02.75
o.B.	1	46101601	46.10.01.16.01.75
16.2+18-20	2	45260110	45.26.01.10.00.75
17	2		46.10.01.19.00.75
18	2(1)	counter sunk screw with hexogon socket DIN 7991 M5x16	01.65.07.92.05.00
19	2(1)	washer DIN 9021 - M5	02.79.05.31.20.20
20	2(1)	hexagon nut self-locking low form DIN 985 - M5	02.32.00.50.01.20


Arm rest for long side guards

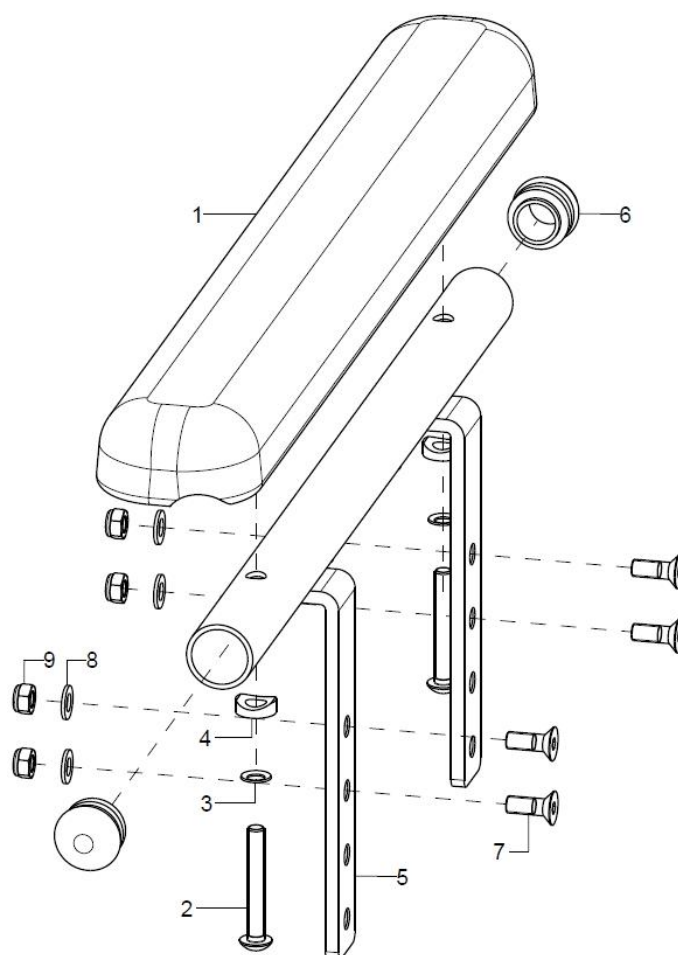


pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-9	1P	452507			
1	2	450010			04.65.01.37.00.10
2-4	1	45250799			
2	2		lens head screw with hexagon socket ISO 7380 - M5x35		01.42.07.99.65.00
3	2		lock washer S5		02.99.10.50.01.10
4	2				40.04.16.01.00.10
5-9	1	45250702			45.25.07.01.02.75
o.B.	1	45250701			45.25.07.01.01.75
6	2		plugs for tube ø20x1-2, black		04.63.10.20.00.10
7	4		counter sunk screw with hexagon socket DIN 7991 M5x14		01.65.07.90.05.00
8	4		washer DIN 125 - A 5,3		02.61.05.31.01.20
9	4		hexagon nut self-locking low form DIN 985 - M5		02.32.00.50.01.20

Desk/long side guard, front socket

Bei abduziertem Rahmen

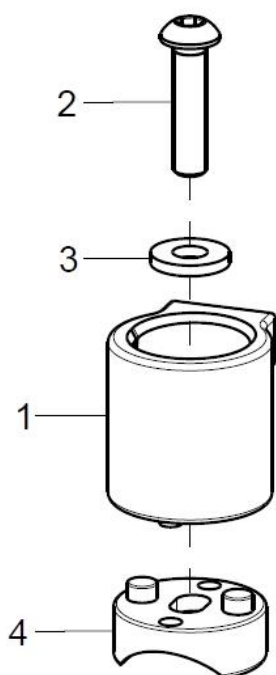


Bei geradem Rahmen

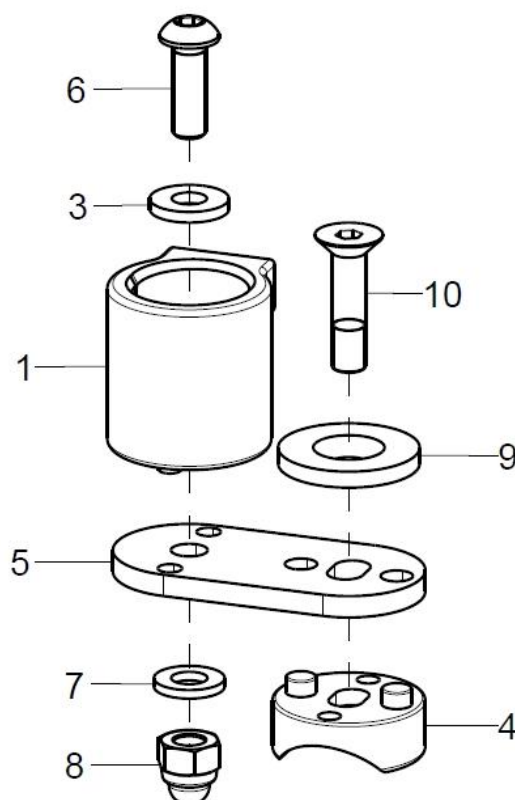


pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-4	2	46100130			
1+3-10	2	46100134			
1	1	46100110			46.10.01.10.00.75
2	1		lens head screw with hexagon socket ISO 7380 - M6x25		01.42.08.56.65.00
3	1		washer reinforced $\varnothing 14/6,2 \times 2$		58.30.15.01.00.24
4	1	46100112			46.10.01.12.00.75
5	1	46100125			46.10.01.25.00.75
6	1		lens head screw with hexagon socket ISO 7380 - M6x18		01.42.08.53.05.00
7	1		washer DIN 125 - A 6,4		02.61.06.41.41.20
8	1		hexagon cap nut low form with clamp DIN 986 - M6		02.33.00.60.11.20
9	1	46100122			46.10.01.22.00.75
10	1		counter sunk screw with hexagon socket DIN 7991 M6x25		01.65.08.56.02.40

Arm pad with raster

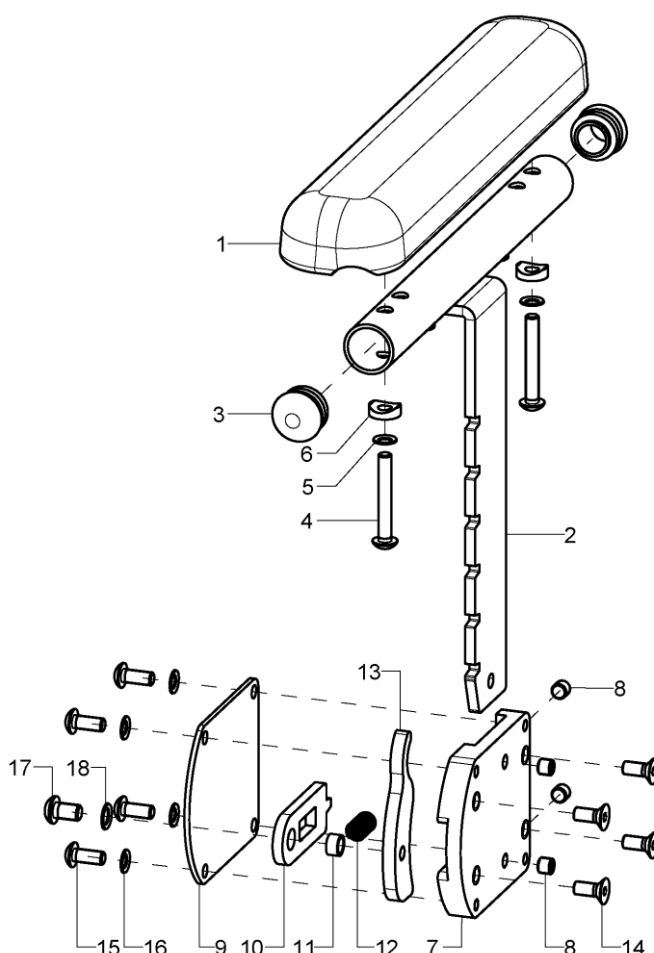


pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-18	1	452506			
1	1	450007	arm rest, straight black 240x50x26, 2xM5x164		04.65.01.24.00.10
2-3	1	45250611			45.25.06.10.02.75
o.B.	1	45250610			45.25.06.10.01.75
3	2		plugs for tube ø20x1-2, black		04.63.10.20.00.10
4	2		lens head screw with hexagon socket ISO 7380 - M5x35		01.42.07.99.65.00
5	2		lock washer S5		02.99.10.50.01.10
6	2				40.04.16.01.00.10
7-18	1	45250622			
o.B.	1	45250621			
7	1				45.25.06.02.00.75
8	4				01.68.08.45.06.00
9	1				45.25.06.04.00.75
10	1				45.25.06.01.00.75
11	1				45.25.06.11.00.75
12	1		pressure spring DIN 2098 0,7x7,65/9,05x21		04.21.00.80.21.00
13	1				45.25.06.05.00.75
14	4		counter sunk screw with hexagon socket DIN 7991 M5x12		01.65.07.89.05.00
15	4		lens head screw with hexagon socket ISO 7380 - M5x12		01.42.07.89.65.00
16	4		lock washer S5		02.99.10.50.01.10
17	1		lens head screw with hexagon socket ISO 7380 - M6x10		01.42.08.48.65.00
18	1		lock washer S6		02.99.10.60.01.10

Side guard for back with gas pressure spring

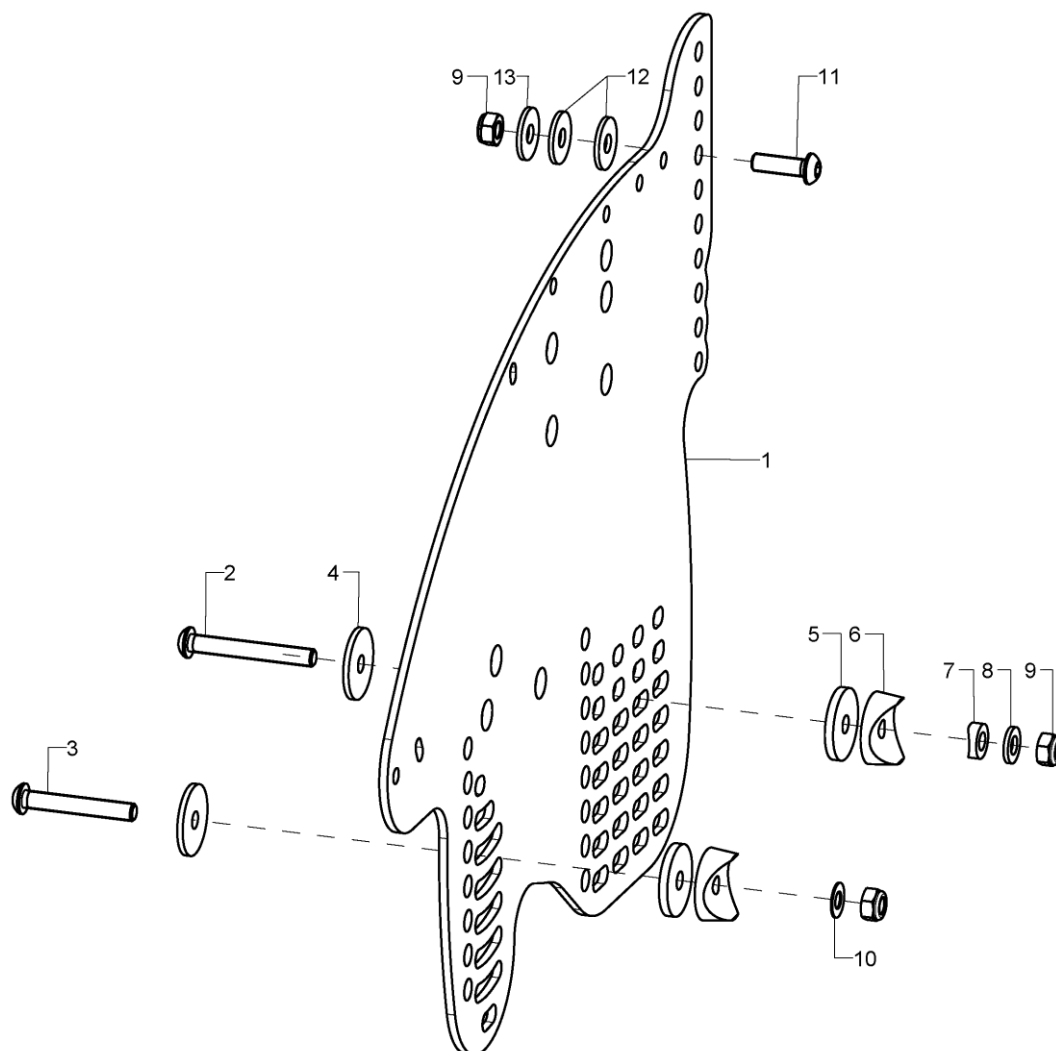


pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1	1	45252604			45.25.05.12.02.75
o.B.	1	45252603			45.25.05.12.01.75
altern.	1	45252404			45.25.05.10.02.75
o.B.	1	45252403			45.25.05.10.01.75
altern.	1	45252204			45.25.05.09.02.75
o.B.	1	45252203			45.25.05.09.01.75
2-13	2	452597			
2	1		lens head screw with hexagon socket ISO 7380 - M6x45		01.42.08.61.65.00
3	1		lens head screw with hexagon socket ISO 7380 - M6x40		01.42.08.60.65.00
4	2				40.24.01.04.00.54
5	2		washer DIN 440, 6,6x22x2		02.67.22.03.09.20
6	2		saddle washer black ø 24/8		04.63.39.24.08.30
7	1		saddle washer ø12/6x2,5		35.04.16.01.00.10
8	1		washer DIN 125 - A 6,4		02.61.06.41.41.20
9	3		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20
10	1		fitted washer DIN 988 - 6x12x0,5 - A2		02.68.06.00.50.30
11	1		lens head screw with hexagon socket ISO 7380 - M6x20		01.42.08.54.05.00
12	2		washer DIN 9021- 6,4- PA		02.79.06.41.60.70
13	1		washer DIN 9021 -6,4		02.79.06.41.60.20

Desk side guard for back with gas pressure spring

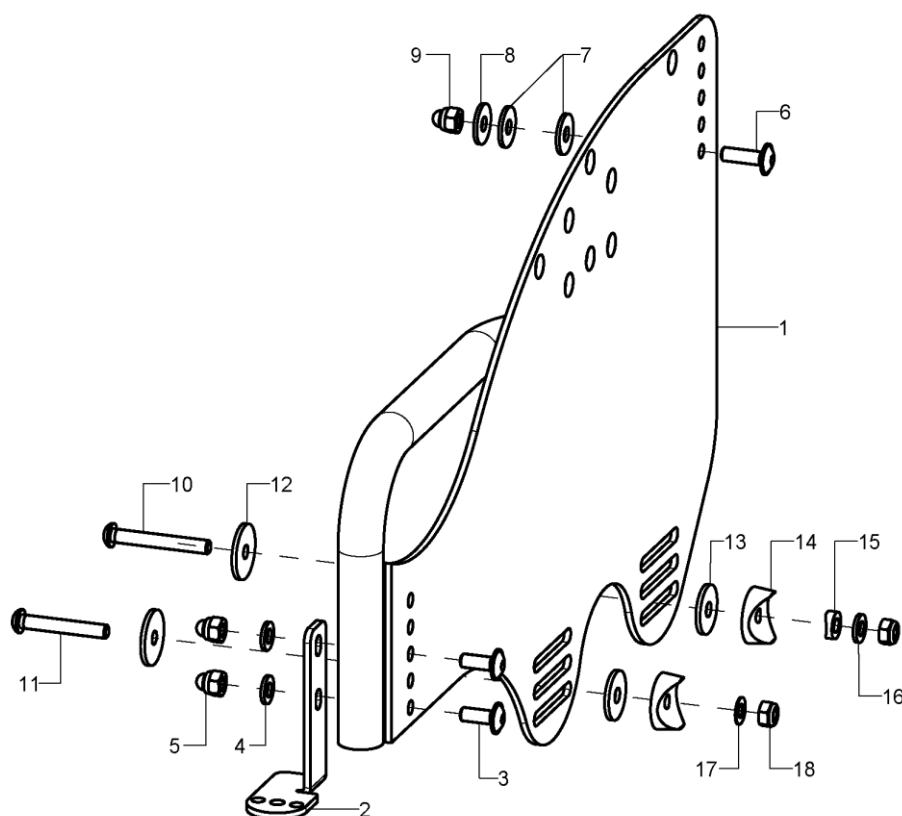


pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1	1	46100402			46.10.04.01.02.75
o.B.	1	46100401			46.10.04.01.01.75
altern.	1	46100502			46.10.03.01.02.75
o.B.	1	46100501			46.10.03.01.01.75
2-5	1	46100404			46.10.04.03.02.75
o.B.	1	46100403			46.10.04.03.01.75
altern.	1	46100604			46.10.06.01.02.75
o.B.	1	46100603			46.10.06.01.01.75
3	2		lens head screw with hexagon socket M6x16		01.43.08.52.65.00
4	2		washer DIN 125 - A 6,4		02.61.06.41.41.20
5	2		hexagon cap nut low form with clamp DIN 986 - M6		02.33.00.60.11.20
6-18	2	46100499			
6	1		lens head screw with hexagon socket M6x20		01.43.08.54.65.00
7	2		washer DIN 9021- 6,4- PA		02.79.06.41.60.70
8	1		washer DIN 9021 -6,4		02.79.06.41.60.20
9	1		hexagon cap nut low form with clamp DIN 986 - M6		02.33.00.60.11.20
10	1		lens head screw with hexagon socket ISO 7380 - M6x45		01.42.08.61.65.00
11	1		lens head screw with hexagon socket ISO 7380 - M6x40		01.42.08.60.65.00
12	2				40.24.01.04.00.54
13	2		washer DIN 440, 6,6x22x2		02.67.22.03.09.20
14	2		saddle washer black ø 24/8		04.63.39.24.08.30
15	1		saddle washer Ø12/6x2,5		35.04.16.01.00.10
16	1		washer DIN 125 - A 6,4		02.61.06.41.41.20
17	1		fitted washer DIN 988 - 6x12x0,5 - A2		02.68.06.00.50.30
18	2		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20

Long side guard for back with gas pressure spring

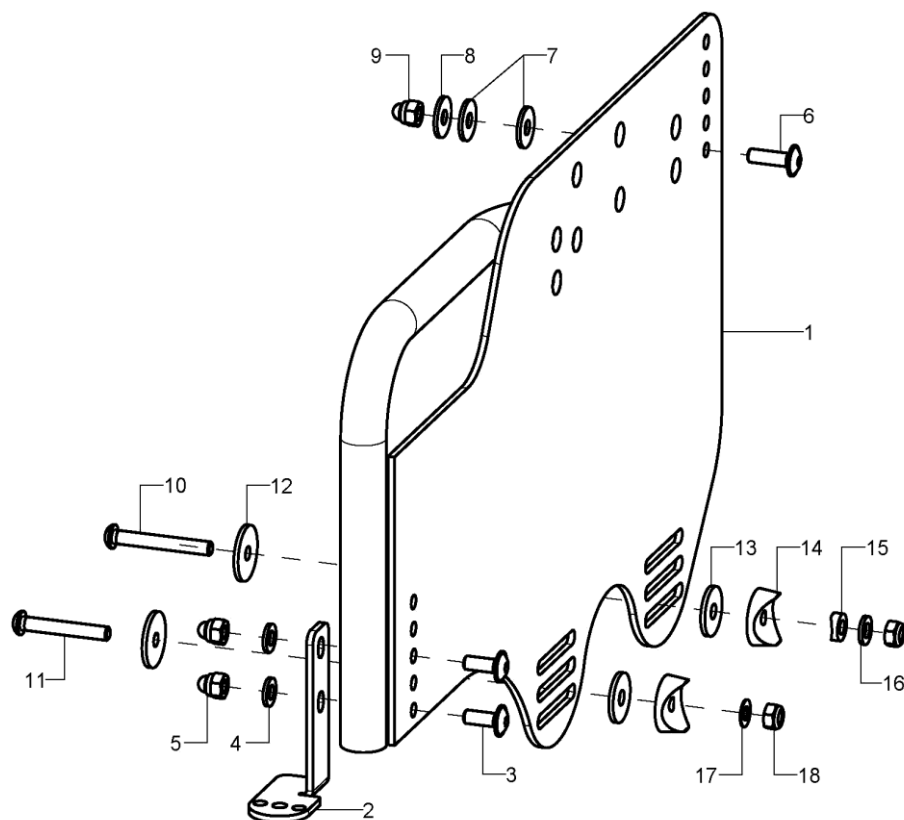

pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1	1	46110402			46.11.04.01.02.75
o.B.	1	46110401			46.10.04.01.01.75
altern.	1	46110502			46.11.05.01.02.75
o.B.	1	46110501			46.11.05.01.01.75
2-5	1	46100404			46.10.04.03.02.75
o.B.	1	46100403			46.10.04.03.01.75
altern.	1	46100604			46.10.06.01.02.75
o.B.	1	46100603			46.10.06.01.01.75
3	2		lens head screw with hexagon socket M6x16		01.43.08.52.65.00
4	2		washer DIN 125 - A 6,4		02.61.06.41.41.20
5	2		hexagon cap nut low form with clamp DIN 986 - M6		02.33.00.60.11.20
6-18	2	46100499			
6	1		lens head screw with hexagon socket M6x20		01.43.08.54.65.00
7	2		washer DIN 9021- 6,4- PA		02.79.06.41.60.70
8	1		washer DIN 9021 -6,4		02.79.06.41.60.20
9	1		hexagon cap nut low form with clamp DIN 986 - M6		02.33.00.60.11.20
10	1		lens head screw with hexagon socket ISO 7380 - M6x45		01.42.08.61.65.00
11	1		lens head screw with hexagon socket ISO 7380 - M6x40		01.42.08.60.65.00
12	2				40.24.01.04.00.54
13	2		washer DIN 440, 6,6x22x2		02.67.22.03.09.20
14	2		saddle washer black ø 24/8		04.63.39.24.08.30
15	1		saddle washer Ø12/6x2,5		35.04.16.01.00.10
16	1		washer DIN 125 - A 6,4		02.61.06.41.41.20
17	1		fitted washer DIN 988 - 6x12x0,5 - A2		02.68.06.00.50.30
18	2		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20

Caster wheel adapter Aluminium

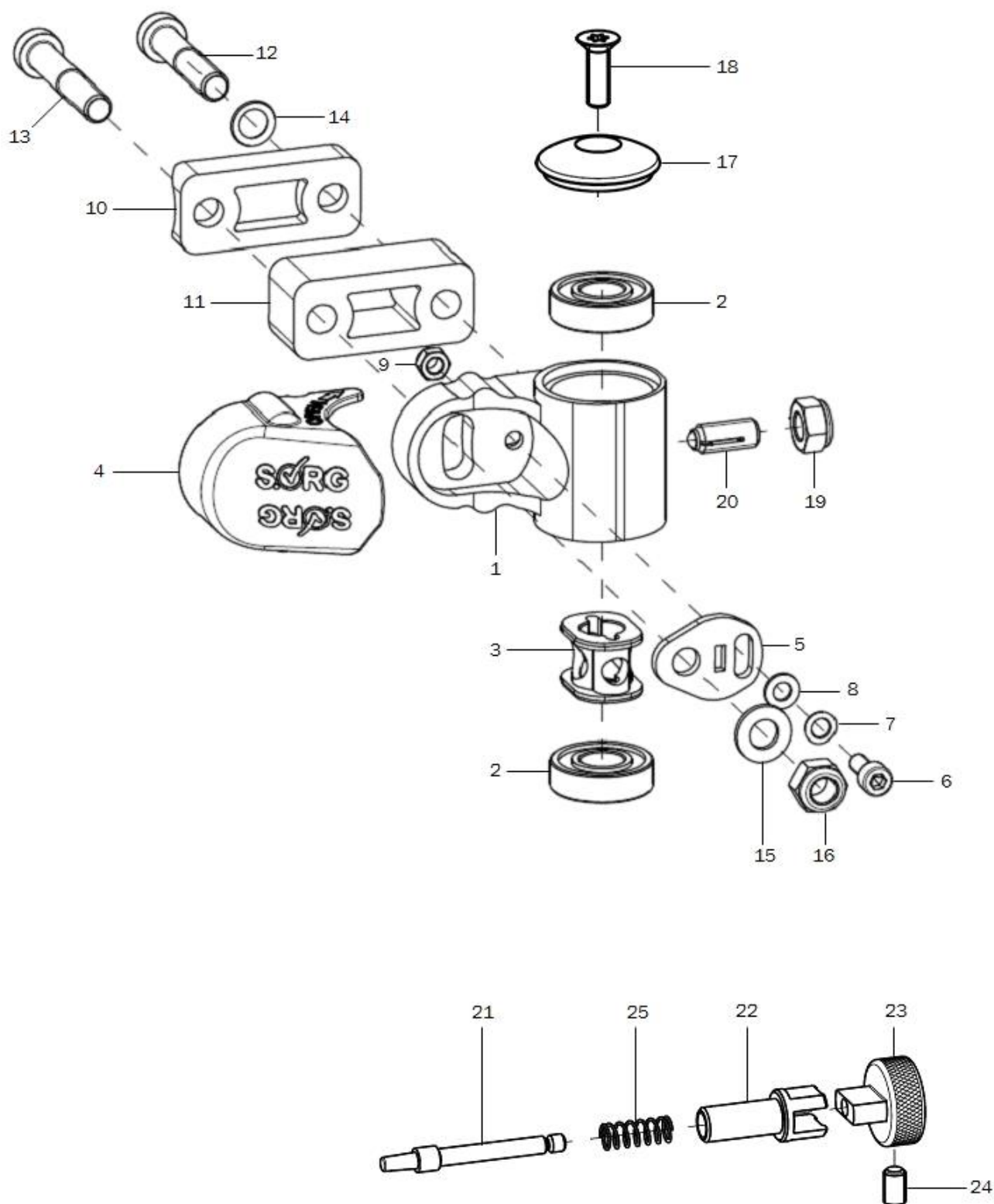



Spare parts list Vector

Caster wheel


Page 88 of 127

pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-9	2	280506	caster adapter track fixation		XXXXXXXXXXXX
1	1	280501	steering adapter aluminum		28.05.01.02.00.00
altern.	2	280500	steering adapter aluminum complete		28.05.01.00.00.00
altern.	1	280501	steering adapter aluminum		28.05.01.02.00.00
2	2	200011	groove ball bearing ø28/12 6001z		04.32.12.08.00.10
3	1	280504	damper element		28.05.01.04.00.00
4	1	280030	cover cap		28.00.10.03.00.10
5	1	280503	eccentric disc		28.05.01.03.00.00
6	1		cylinder screw DIN 912 M5x12		01.22.07.89.72.40
7	1		lock washer S5		02.99.10.50.01.10
8	1		washer DIN 125 - A 5,3		02.61.05.31.01.20
9	1		hexagon nut self-locking low form DIN 985 - M5		02.32.00.50.01.20
10+12-16	2	280510	mounting set caster adapter aluminum		28.05.10.90.01.00
10-16	2	280511	mounting set caster adapter aluminum 20mm		28.05.10.90.02.00
10-16	2	280512	mounting set caster adapter aluminum 40mm		28.05.10.90.03.00
10+12-16	2	280550	change set caster adapter		
10-16	2	280551	change set caster adapter 20mm		
10-16	2	280552	change set caster adapter 40mm		
10	1	280502	half shell LRAD R12		28.05.02.01.00.00
11	1	280513	track extention 20mm		28.05.02.03.00.00
altern.	1	280514	track extention 40mm		28.05.02.04.00.00
12	1		cylinder screw low head DIN 7984 - M8x40		01.24.09.80.62.40
altern.	1		cylinder screw low head DIN 7984 - M8x60		01.24.09.84.02.40
altern.	1		cylinder screw low head DIN 7984 - M8x80		01.24.09.88.62.40
altern.	1				
altern.	1				
altern.	1				
13	1		cylinder screw low head DIN 7984 - M8x45		01.24.09.81.62.40
altern.	1		cylinder screw low head DIN 7984 - M8x65		01.24.09.85.62.40
altern.	1		cylinder screw low head DIN 7984 - M8x85		01.24.09.89.62.40
altern.	1				
altern.	1				
altern.	1				
14	1		lock washer S8		02.99.10.80.01.10
15	1		washer DIN 125 - A 8,4		02.61.08.41.41.20
16	1		hexagon nut self-locking low form DIN 985 - M8		02.32.00.80.01.20
17	1	282107	cap 35x10 black anodized incl. mounting screw		28.21.07.01.00.10
18	1		counter sunk screw with Torx ISO 14581 M6x20 vz, black		01.66.08.54.92.21
19-20		285103	spring pressure piece M8 complete		04.14.50.80.21.20
20	1		hexagon nut self-locking low form DIN 985 - M8		02.32.00.80.01.20
21-25	2	285003	track fixation complete		XXXXXXXXXXXX
21	1				28.50.01.01.00.10
22	1				28.50.01.02.00.10
23	1				28.50.01.03.00.10
24	1		setscrew with hexagon socket DIN 913,M5x8		01.67.07.87.61.20
25	1		pressure spring 0,55x5,5x23,7		04.21.05.52.37.10

Achtung: Bei der Umrüstung von Kunststoff auf Aluminium-Lenkradadapter bitte paarweise bestellen.


Wenn eine Lenk- und Schiebehilfe oder Rückhalteösen am Rollstuhl angebaut sind, bitte bei Bestellungen von Ersatzteilen angeben.

Adapter for low seat heights

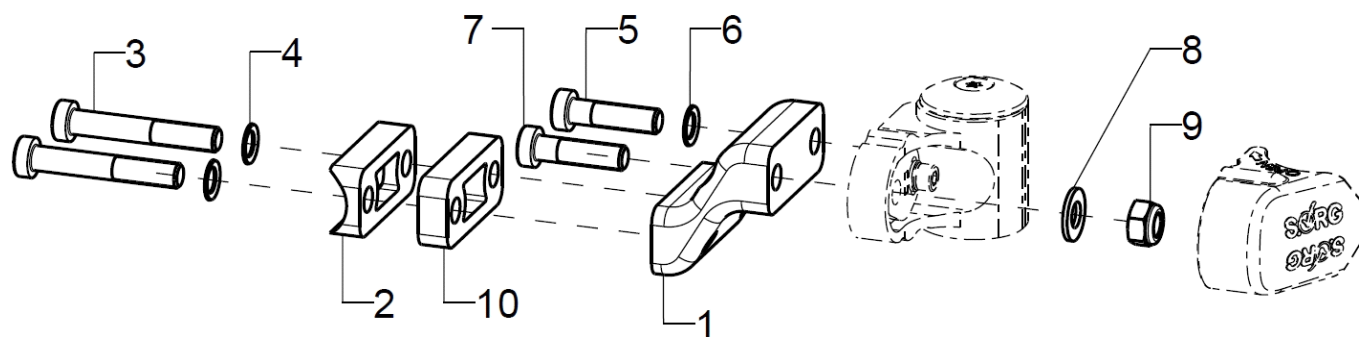

pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1	2	280525			28.05.03.01.00.00
altern.	2	280526			28.05.03.02.00.00
2-9	2	280527			
2	1	280502	half shell LRAD R12		28.05.02.01.00.00
3	2		cylinder screw low head DIN 7984 - M8x40		01.24.09.80.62.40
altern.	2				01.24.09.82.62.40
4	2		lock washer S8		02.99.10.80.01.10
5	1				01.24.09.78.62.40
6	1		lock washer S8		02.99.10.80.01.10
7	1				01.24.09.79.62.40
8	1		washer DIN 125 - A 8,4		02.61.08.41.41.20
9	1		hexagon nut self-locking low form DIN 985 - M8		02.32.00.80.01.20
10 (optional)	1	280515			28.05.02.05.00.00

Caster wheel fork

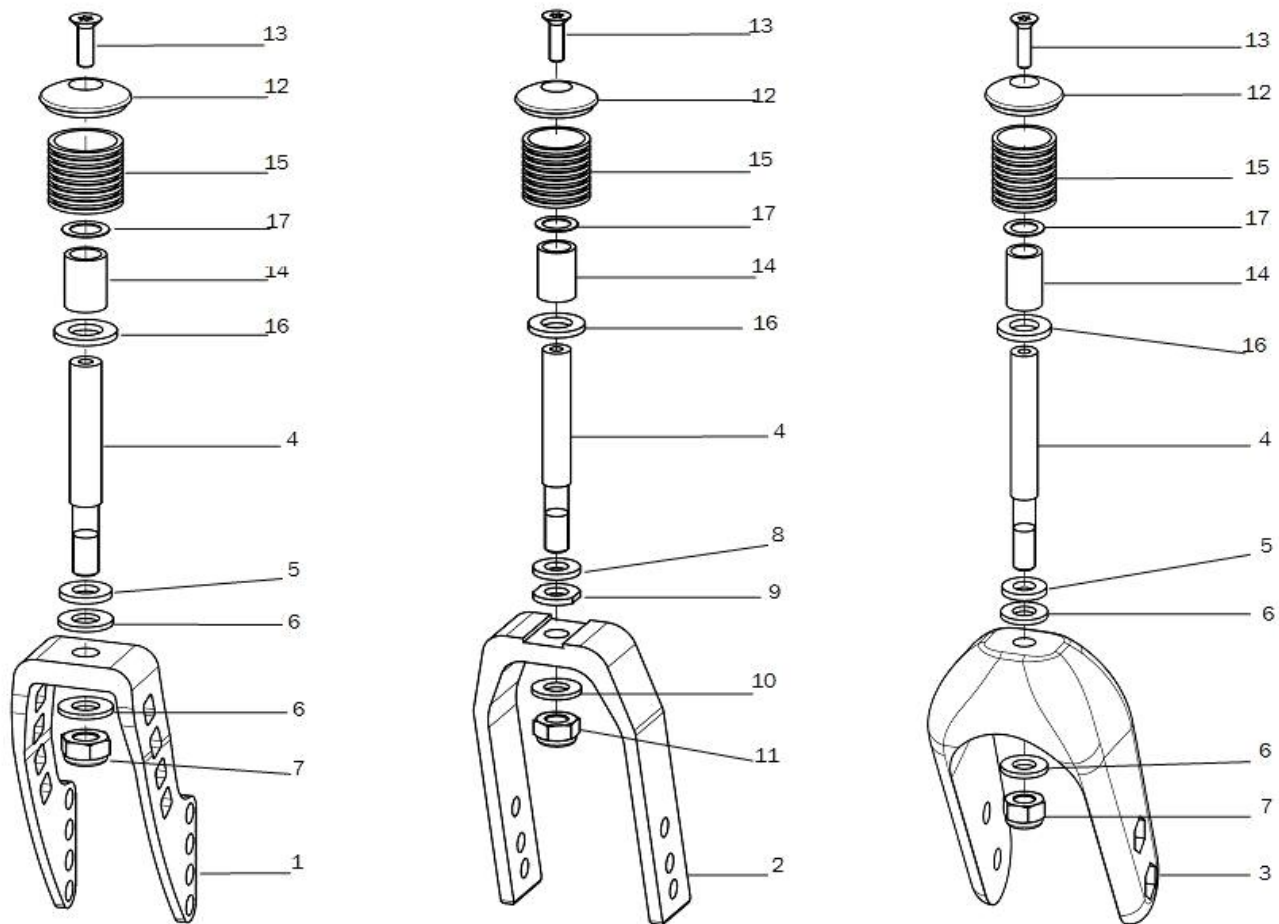

pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1	2	2841	caster fork aluminum 164x45		28.41.01.00.00.10
o.B.	2	2842	caster fork aluminum 126x45		28.42.01.00.00.10
o.B.	2	2843	caster fork aluminum 100x45		28.43.01.00.00.10
2	2	2821	caster fork, aluminum		28.00.02.01.00.10
3	2	2823	caster fork large, pu 60x130		04.52.59.60.90.23
o.B.	2	2822	caster fork small, pu 32x120		04.52.59.32.90.23
4	2	282104	steering axle $\varnothing 12 \times 65$, 1		28.21.05.01.00.10
o.B.	2	285101	steering axle 65 for track fixation		28.50.00.03.00.10
o.B.	2	285001	steering axle 65 for locking		28.50.00.01.00.10
o.B.	2	282105	steering axle 90, $\varnothing 12 \times 89$, 8		28.21.05.02.00.10
o.B.	2	285102	steering axle 90 for track fixation		28.50.00.04.00.10
o.B.	2	285002	steering axle 90 for locking		28.50.00.02.00.10
5-7	2	282120	mounting set caster axle on caster fork aluminum (washers, hexagon nut)		28.21.20.01.01.00
5	1		washer DIN 1440 -10x20x2,5		02.71.10.02.50.20
6	2		washer DIN 1440 -10x20x2,5		02.71.10.02.50.20
7	1		hexagon nut self-locking low form DIN 985 - M10		02.32.01.00.01.20
8-11	2	282121	mounting set steering axle wide		XXXXXXXXXXXX
8	1		washer DIN 1440 -10x20x2,5		02.71.10.02.50.20
9	1		washer DIN 1440 -10x20x2,5		02.71.10.02.50.20
10	1		washer DIN 1440 -10x20x2,5		02.71.10.02.50.20
11	1		hexagon nut self-locking low form DIN 985 - M10		02.32.01.00.01.20
12	2	282107	cap 35x10 black anodized incl. mounting screw		28.21.07.01.00.10




Spare parts list Vector

Caster wheel

o.B.	2	270005	cap Rd 32x10	28.21.07.02.00.10
13	2		counter sunk screw with Torx ISO 14581 M6x20 vz, black	01.66.08.54.92.21
14-17	2	282123	mounting set caster axle 90mm	XXXXXXXXXXXX
14	1			19.22.03.08.00.47
15	1			Ohne Nummer
16	1		washer DIN 125 - B 13	02.61.13.02.32.20
17	1		fitted washer DIN 988 - 12x18x0,5	02.68.12.00.50.30

Caster wheel fork with quick-release axle

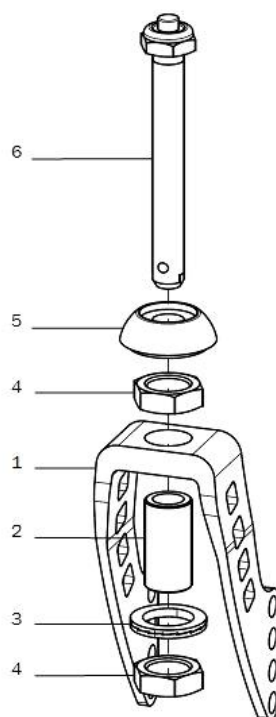


pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1	2	272114	caster fork 164x45 for quick-release axle		27.21.01.14.00.10
o.B.	2	272113	caster fork 126x45 for quick-release axle		27.21.01.13.00.10
o.B.	2	272112	caster fork 100x45 for quick-release axle		27.21.01.12.00.10
2	2	272102	adapter M18x1 D12 35mm		27.21.01.02.00.10
3-4	2	272104	mounting set adapter		XXXXXXXXXXXX
3	1		lock washer M18 (Nord-Lock)		02.99.21.95.01.10
4	2		hexagon nut M18x1		02.19.01.30.11.21
5	1	272101	cap 27x16,5 alumimum, black anodized		27.21.01.01.00.10
6	1	272103	quick-release axle 12x85		04.51.51.21.40.30



Caster wheels


pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
o.B.	2	2812	caster 4", PU		
o.B.	2	2804	steering wheel 4 solid rubber		
o.B.	2	2820	caster 4" PU with LED		
o.B.	2	2835	caster 4" solid rubber black with aluminum rim		04.52.41.04.00.31
o.B.	2	2805	caster 5" PU		
o.B.	2	2809	caster 5" soft solid rubber		
o.B.	2	2834	caster 5" PU, transparent with LED		
o.B.	2	2836	caster 5" solid rubber black with aluminum rim		04.52.41.05.00.61
o.B.	2	2806	caster 140 mm soft, hub width 46		
o.B.	2	2803	caster 6" PU		
o.B.	2	2811	caster 6" solid rubber		
o.B.	2	2808	caster 6" pneumatic wheels		
o.B.	2	2838			04.52.41.06.00.70
o.B.	2	2813	caster 7" solid rubber		
o.B.	2	2807	caster 7" pneumatic wheels		
o.B.	2	289998			

Axle plate standard

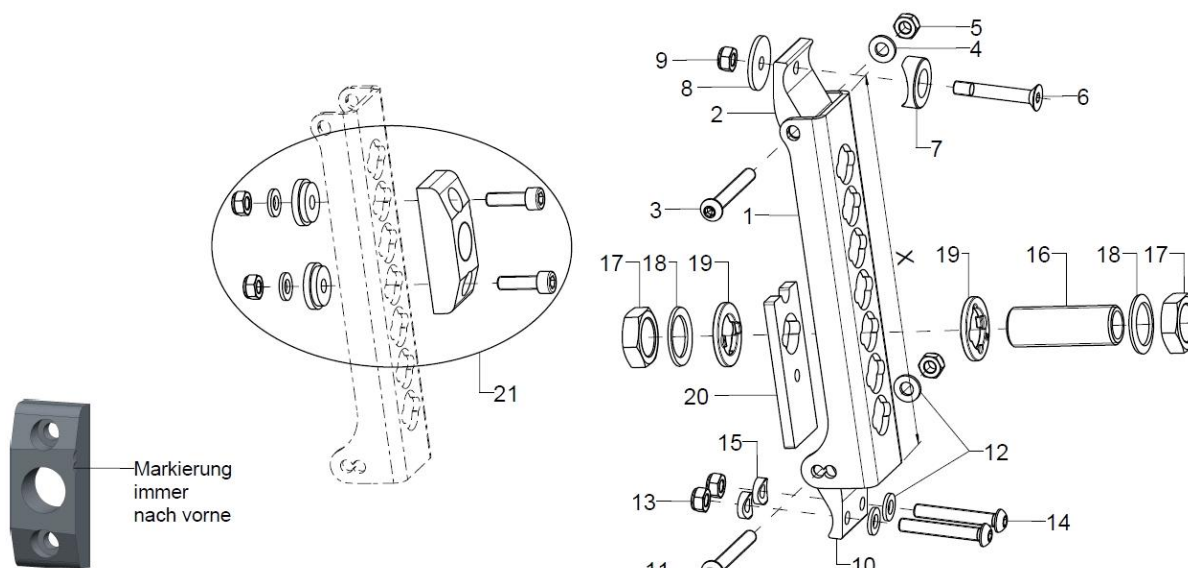


pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1	2	20150902	axle plate, X = 176,5 mm		20.15.09.11.00.69
o.B.	2	20150901	axle plate, X = 156,5 mm		20.15.09.10.00.69
2	2	20150903	attachment 1B		20.15.09.12.00.69
3-9	2	20150999	mounting set attachment 1B		XXXXXXXXXXXX
3	1		lens head screw with hexagon socket ISO 7380 - M6x50		01.42.08.62.65.00
4	1		washer DIN 125 - A 6,4		02.61.06.41.41.20
5	1		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20
6	1		counter sunk screw with hexagon socket DIN 7991 M6x45 / 9+1 - 8.8 vz		01.65.08.61.02.41
7	1				20.15.09.05.00.69
8	1				40.24.01.04.00.54
9	1		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20
10	2	20150913	attachment 2B		20.15.09.13.00.69
o.B.	2	20150917	attachment for drum brake 2B		20.15.09.17.00.69
11-15	2	20150998	mounting set attachment 2B		XXXXXXXXXXXX
11	1		lens head screw with hexagon socket ISO 7380 - M6x50		01.42.08.62.65.00
12	3		washer DIN 125 - A 6,4		02.61.06.41.41.20
13	3		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20
14	2		lens head screw with hexagon socket ISO 7380 - M6x40		01.42.08.60.65.00
15	2				40.04.16.01.00.10
16-20	1	210019	axle adapter M18x1 d12,7 incl. mounting set		04.15.11.35.00.30
o.B.	1	210024	axle adapter M18x1 d12		04.51.51.25.00.31
17	2		hexagon nut M18x1		02.19.01.30.11.21
18	2		lock washer S18		02.99.11.80.01.10
19	2	20150501			20.15.05.01.01.00
o.B.	2	20150502			20.15.05.02.01.00
20	1	20150921	reinforcement axle plate		20.15.09.21.00.69
21	2	20150929	distance block for wheel base extension incl. mounting set		XXXXXXXXXXXX


Einstellungen Radsturz bei Standard - Antriebsrädern

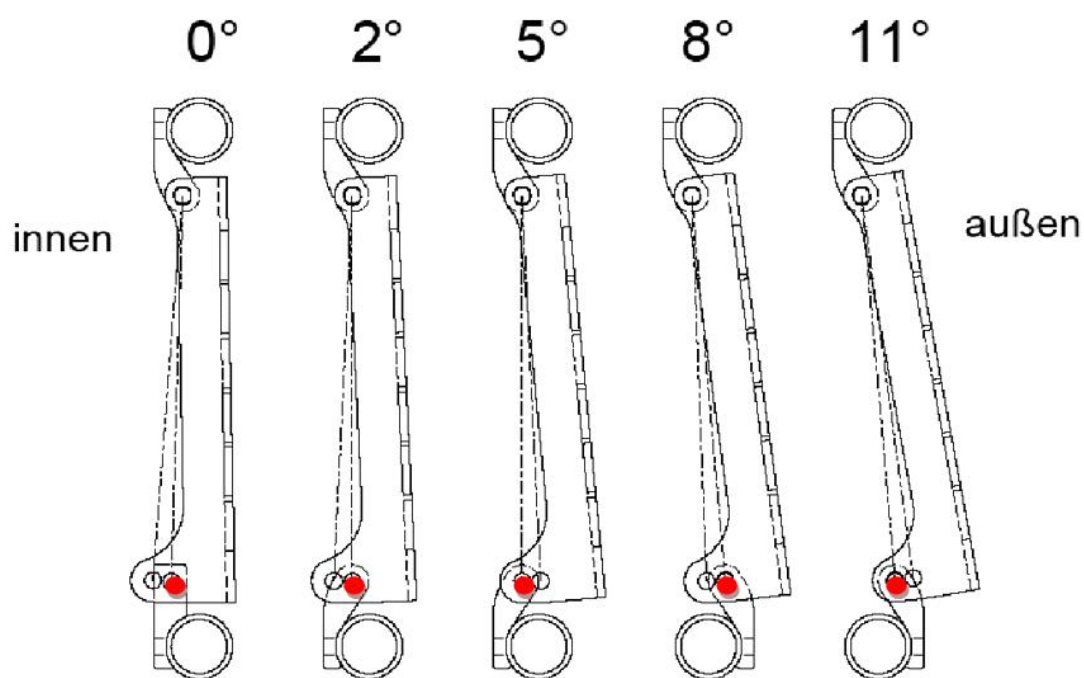

Einstellungen Radsturz bei Trommelbremsrädern - Stand: 25.02.14

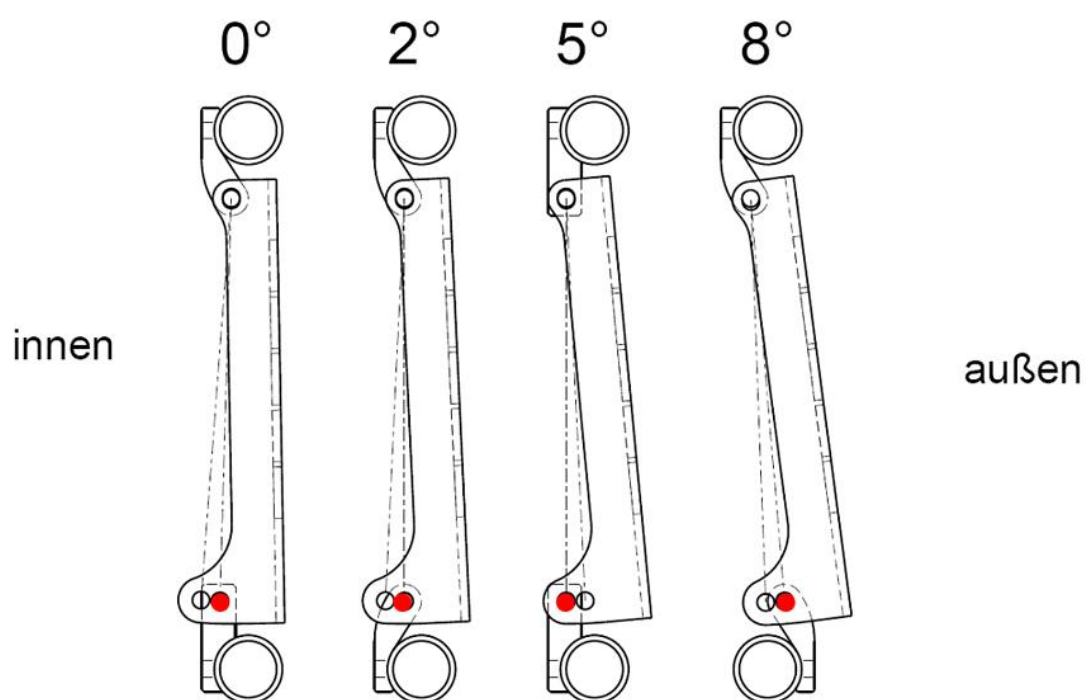


Axle plate drum brake

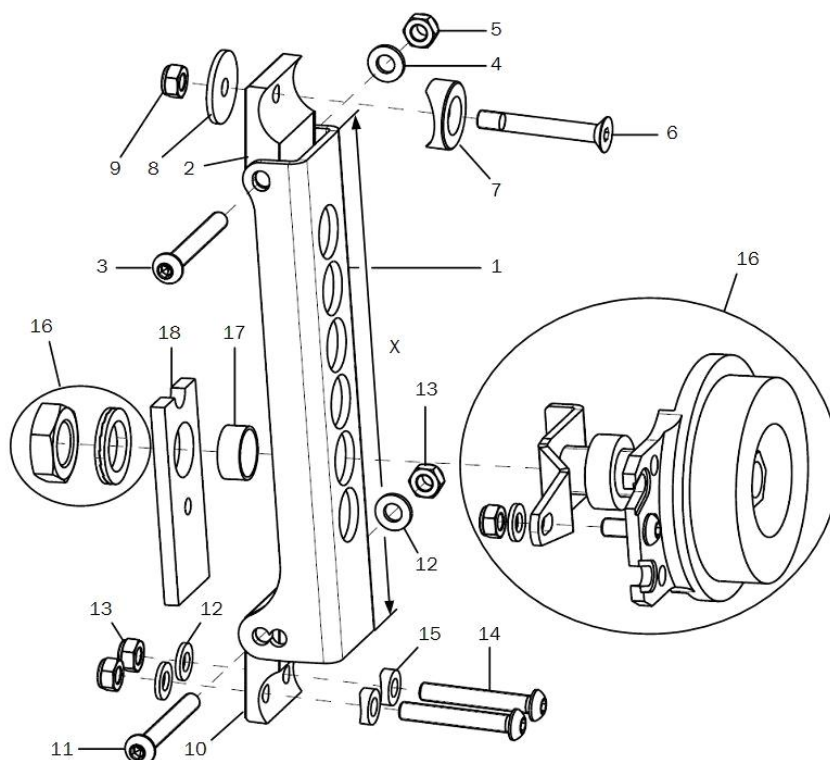

pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1	2	20150901	axle plate, X = 156,5 mm		20.15.09.10.00.69
o.B.	2	20150902	axle plate, X = 176,5 mm		20.15.09.11.00.69
2	2	20150916	attachment for drum brake 1B		20.15.09.06.00.69
o.B.	2	20150903	attachment 1B		20.15.09.12.00.69
3-9	2	20150999	mounting set attachment 1B		XXXXXXXXXXXX
3	1		lens head screw with hexagon socket ISO 7380 - M6x50		01.42.08.62.65.00
4	1		washer DIN 125 - A 6,4		02.61.06.41.41.20
5	1		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20
6	1		counter sunk screw with hexagon socket DIN 7991 M6x45 / 9+1 - 8.8 vz		01.65.08.61.02.41
7	1				20.15.09.05.00.69
8	1				40.24.01.04.00.54
9	1		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20
10	2	20150917	attachment for drum brake 2B		20.15.09.17.00.69
o.B.	2	20150913	attachment 2B		20.15.09.13.00.69
11-15	2	20150998	mounting set attachment 2B		XXXXXXXXXXXX
11	1		lens head screw with hexagon socket ISO 7380 - M6x50		01.42.08.62.65.00
12	3		washer DIN 125 - A 6,4		02.61.06.41.41.20
13	3		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20
14	2		lens head screw with hexagon socket ISO 7380 - M6x40		01.42.08.60.65.00
15	2				40.04.16.01.00.10
16	1	220005	drum brake body complete, incl. adapter and counter wrench		22.00.01.99.00.10
o.B.	1	220013	drum brake anchor pre-assembled		
17	2	22001202			22.00.12.04.00.69
18	1	20150921	reinforcement axle plate		20.15.09.21.00.69
o.B.	2	597492	extension set for drum brake wheel		XXXXXXXXXXXX

Wheel base lengthening

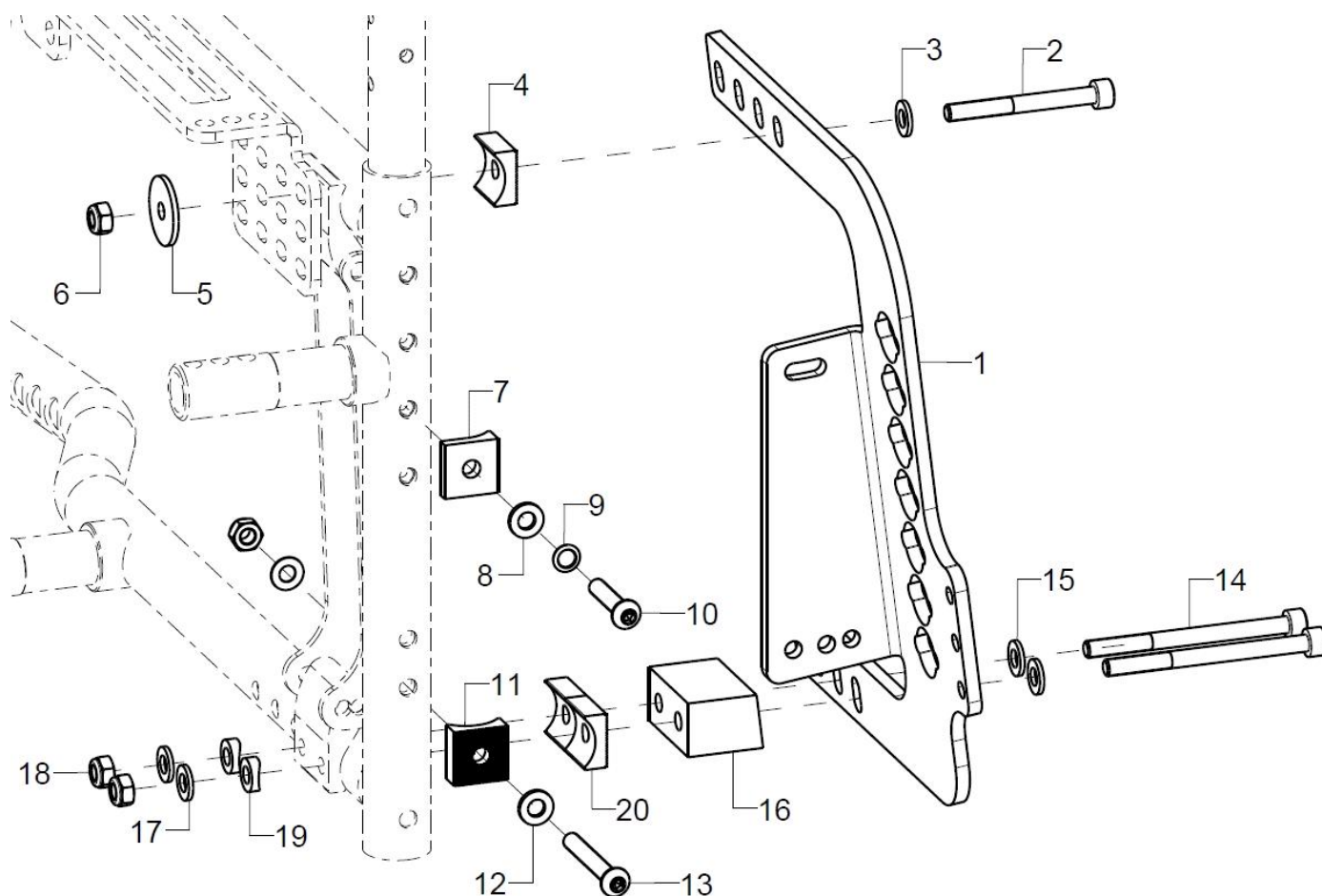

pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-20	1P	7601	wheel base extension		
1-20	1	76060101	wheel base extension right		76.06.02.01.02.00
o.B.	1	76060102	wheel base extension left		76.06.02.01.01.00
altern.	1	76060202			76.06.02.02.02.00
o.B.	1	76060201			76.06.02.02.01.00
altern.	1	76060302			76.06.02.03.02.00
o.B.	1	76060301			76.06.02.03.01.00
2	1				01.22.08.63.62.40
3	1		washer DIN 125 - A 6,4		02.61.06.41.41.20
4	1				76.01.03.01.00.10
5	1				40.24.01.04.00.54
6	1		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20
7	1	207108	half shell 20x20x8,5		20.71.01.08.00.71
8	1		washer DIN 125 - A 6,4		02.61.06.41.41.20
9	1		lock washer S6		02.99.10.60.01.10
10	1		lens head screw with hexagon socket ISO 7380 - M6x30		01.42.08.58.65.00
11	1				76.01.03.01.00.10
12	1		washer DIN 125 - A 6,4		02.61.06.41.41.20
13	1		lens head screw with hexagon socket ISO 7380 - M6x45		01.42.08.61.65.00
14	2				01.22.08.61.62.40
altern.	2				01.22.08.66.02.40
altern.	2				01.22.08.67.02.40
15	2		washer DIN 125 - A 6,4		02.61.06.41.41.20




16-20	1	76060103	76.06.02.04.01.00
altern.	1	76060104	76.06.02.04.02.00
17	2	washer DIN 125 - A 6,4	02.61.06.41.41.20
18	2	hexagon nut self-locking low form DIN 985 - M6	02.32.00.60.01.20
19	2	saddle washer Ø12/6x2,5	35.04.16.01.00.10
20	2		76.01.03.01.00.10

Achtung:


Bei dem Anbau der Radstandsverlängerung ist auch der Anbau der Bremsschiene für die Radstandsverlängerung erforderlich. Siehe Baugruppe Rahmen.

Rear wheels

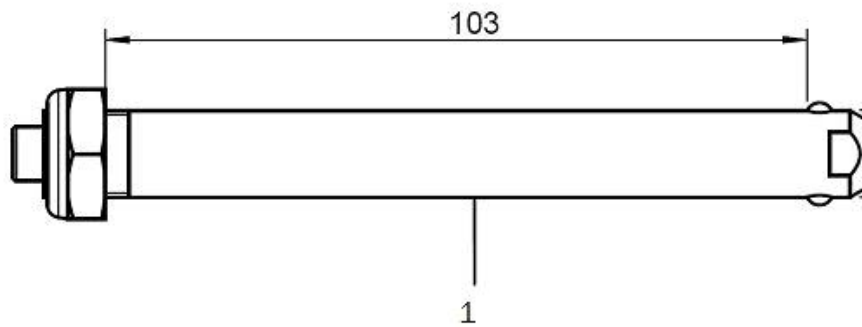


pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
o.B.	2	212200	22" driving wheel with standard hub		
o.B.	2	212400	24" driving wheel standard hub		
o.B.	2	222200	22" driving wheel with drum brake		
o.B.	2	222400	24" driving wheel standard with drum brake		
o.B.	2	212240	22" profile wheel with built-in hand rim, standard hub		
o.B.	2	212440	24" profile wheel with built-in hand rim, standard hub		
o.B.	2	222240	22" profile wheel with built-in hand rim with drum brake		
o.B.	2	222440	24" profile wheel with built-in hand rim with drum brake		
o.B.	2	212277	22" lightweight wheel with hand rim, standard		
o.B.	2	212477	24" lightweight wheel with hand rim, standard		
o.B.	2	212281	22" lightweight wheel with drum brake, black		
o.B.	2	212481	24" lightweight wheel with drum brake, black		
1	2	220001	quick-release axle D12x123mm for drum brake		04.51.51.21.20.30
o.B.	2	210003	quick-release axle 12,7x103 mm for standard hub from 5/2013 and light weight wheel		04.51.51.21.00.31
o.B.	2	200011	groove ball bearing $\varnothing 28/12$ 6001z		04.32.12.08.00.10
o.B.	2	210007	groove ball bearing D28/12,7 6001RS		
o.B.	2	210008	groove ball bearing $\varnothing 28,53/12,7$		04.32.13.08.00.10

Double handrim standard

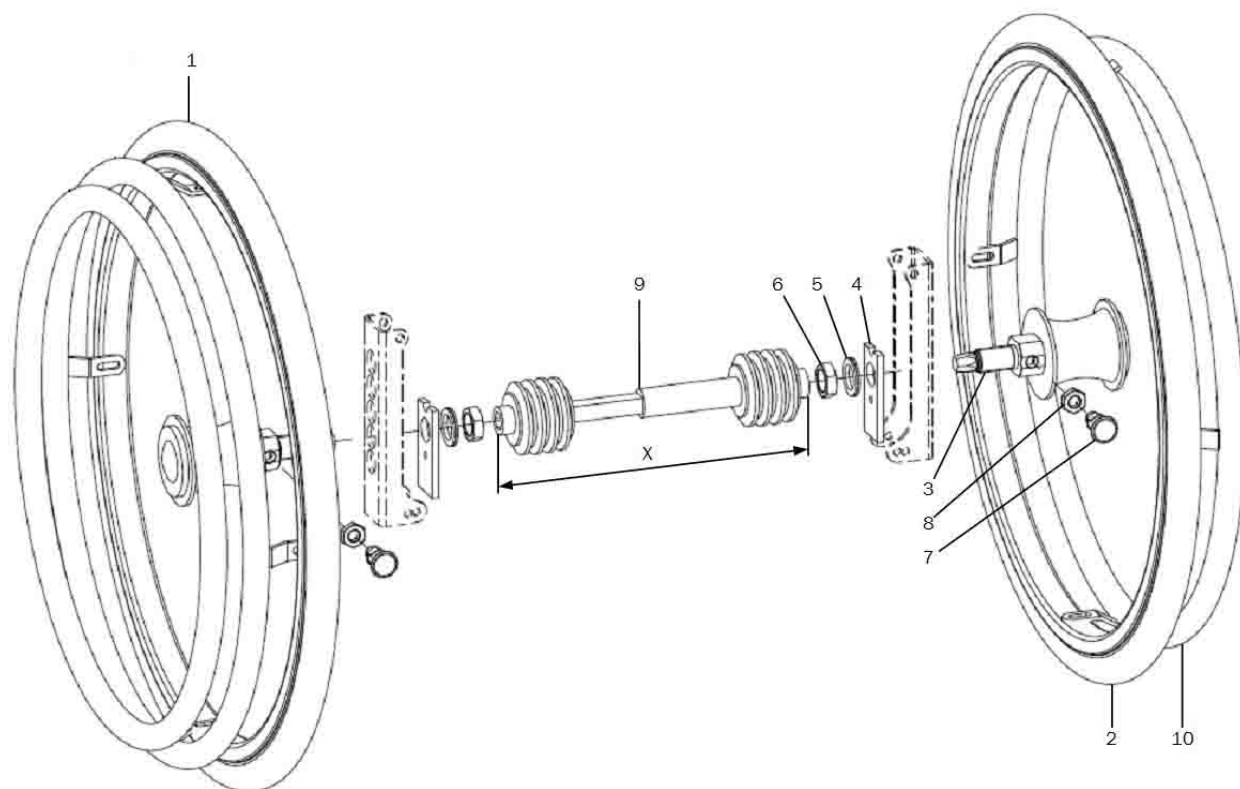

pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-9	1	261403	double handrim 22" standard hub incl. drag anchor, brake arm and telescope rod		XXXXXXXXXX
o.B.	1	261405	double handrim 24" standard hub incl. drag anchor, brake arm and telescope rod		XXXXXXXXXX
1+3-9	1	261442	22" double handrim driving wheel standard hub, driving side		XXXXXXXXXX
o.B.	1	261444	24" double handrim driving wheel standard hub, driving side		XXXXXXXXXX
2-8	1	261443	22" double handrim driving wheel standard hub, opposite side		XXXXXXXXXX
o.B.	1	261445	24" double handrim driving wheel standard hub, opposite side		XXXXXXXXXX
3-8	1	261429	mounting set DHR drum brake 3-8		XXXXXXXXXX
3	1				
4	1	20150921	reinforcement axle plate		20.15.09.21.00.69
5	1		lock washer M18 (Nord-Lock)		02.99.21.95.01.10
6	1		hexagon nut M18x1		02.19.01.30.11.21
7	1		latch ø6/M10x1x31 with lock		04.14.43.60.02.00
8	1		hexagon nut DIN 934 - M10		02.19.01.00.01.20
9	1	2614XX	telescoping tube for double handrim		
10	1	261431	hand rim for other side 22"		XXXXXXXXXX
o.B.	1	261432	hand rim for other side 24"		XXXXXXXXXX

Double handrim drum brake

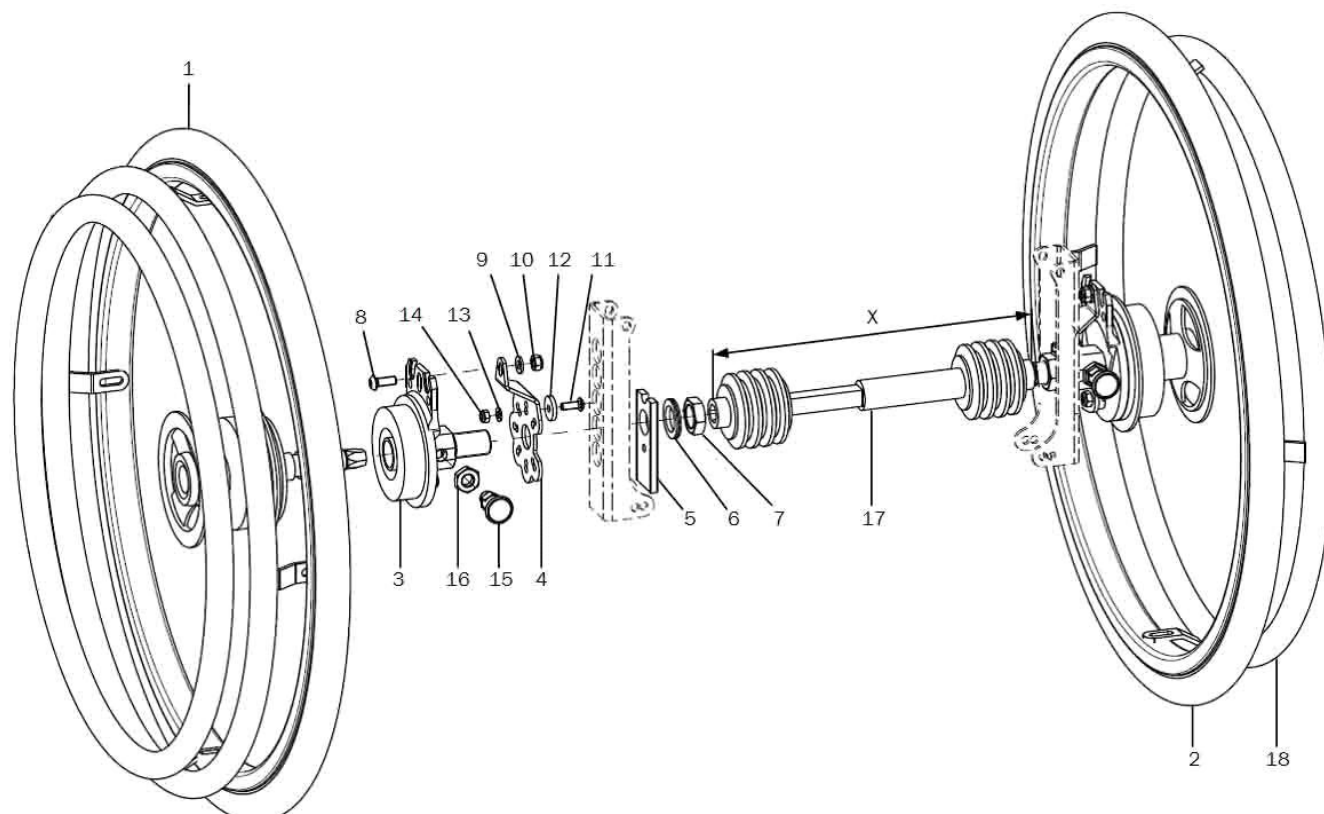

pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-17	1	261404	double handrim 22" drum brake hub incl. drag anchor, brake arm and telescope rod		XXXXXXXXXXXX
o.B.	1	261406	double handrim 24" drum brake hub incl. drag anchor, brake arm and telescope rod		XXXXXXXXXXXX
1	1	261452	22" double handrim driving wheel drum brake hub, driving side		XXXXXXXXXXXX
o.B.	1	261454	24" double handrim driving wheel drum brake hub, driving side		XXXXXXXXXXXX
2	1	261453	22" double handrim driving wheel drum brake hub, opposite side		XXXXXXXXXXXX
o.B.	1	261455	24" double handrim driving wheel drum brake hub, opposite side		XXXXXXXXXXXX
3-16	1	261428	mounting set DHR drum brake		XXXXXXXXXXXX
3	1				
4	1				
5	1	20150921	reinforcement axle plate		20.15.09.21.00.69
6	1		lock washer M18 (Nord-Lock)		02.99.21.95.01.10
7	1		hexagon nut M18x1		02.19.01.30.11.21
8	1		lens head screw with hexagon socket ISO 7380 - M6x16		01.42.08.52.65.00
9	1		washer DIN 125 - A 6,4		02.61.06.41.41.20
10	1		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20
11	1		lens head screw with hexagon socket ISO 7380 - M5x14		01.42.07.90.65.00
12	1				
13	1		washer DIN 125 - A 5,3		02.61.05.31.01.20
14	1		hexagon nut self-locking low form DIN 985 - M5		02.32.00.50.01.20
15	1		latch ø6/M10x1x31 with lock		04.14.43.60.02.00
16	1		hexagon nut DIN 934 - M10		02.19.01.00.01.20



17	1	2614XX	telescoping tube for double handrim	
18	1	261431	hand rim for other side 22"	XXXXXXXXXXXX
o.B.	1	261432	hand rim for other side 24"	XXXXXXXXXXXX



Telescopic rod

pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
o.B.	1	261410	telescope rod 170 - 195 mm		
o.B.	1	261412	telescope rod 195 - 235 mm		
o.B.	1	261413	telescope rod 220 - 275 mm		
o.B.	1	261414	telescope rod 255 - 315 mm		
o.B.	1	261415	telescope rod 295 - 370 mm		
o.B.	1	261416	telescope rod 350 - 460 mm		



Tyres

pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
o.B.	2	202013	tire for pneumatic tube 20x1" Right Run, grey/black		
o.B.	2	202213	tire for pneumatic tube 22x1" Right Run, grey/black		
o.B.	2	202413	tire for pneumatic tube 24x1" Right Run, grey/black		
o.B.	2	202021	tire for pneumatic tube 20x1 3/8" grey		
o.B.	2	202221	tire for pneumatic tube 22x1 3/8" grey		
o.B.	2	202421	tire for pneumatic tube 24x1 3/8" grey		
o.B.	2	202414	tire for pneumatic tube 24x1" Right Run blue/black		
o.B.	2	202415	tire for pneumatic tube 24x1" Right Run red/black		
o.B.	2	232011	puncture proof tires 20x1"		
o.B.	2	232211	puncture proof tires 22x1"		
o.B.	2	232411	puncture proof tires 24x1"		
o.B.	2	232021	puncture proof tires 20x1 3/8"		
o.B.	2	232221	puncture proof tires 22x1 3/8"		
o.B.	2	232421	puncture proof tires 24x1 3/8"		
o.B.	2	232012			04.61.22.00.10.11
o.B.	2	232212			04.61.22.20.10.11
o.B.	2	232412			04.61.22.40.10.11
o.B.	2	202212	tire for pneumatic tube 22x1" Marathon plus, black		
o.B.	2	202412	tire for pneumatic tube 24x1" Marathon plus, black		
o.B.	2	200520	tube 20"		
o.B.	2	200522	tube 22"		
o.B.	2	200524	tube 24"		




Handrims

pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
o.B.	2	302101	handrim 20" high, stainless steel polished		
o.B.	2	302201	handrim 22" standard, stainless steel polished		
o.B.	2	302301	handrim 22" high, stainless steel polished		
o.B.	2	302401	handrim 24" standard, stainless steel polished		
o.B.	2	302501	handrim 24" high, stainless steel polished		
o.B.	2	302102	handrim 20" high, aluminum coated		
o.B.	2	302202	handrim 22" standard, aluminum coated		
o.B.	2	302302	handrim 22" high, aluminum coated		
o.B.	2	302402	handrim 24" standard, aluminum coated		
o.B.	2	302502	handrim 24" high, aluminum coated		
o.B.	2	3420	Maxgrepp-handrim 20", 4 loops complete		
o.B.	2	3422	Maxgrepp-handrim 22", 4 loops complete		
o.B.	2	3424	Maxgrepp-handrim 24", 4 loops complete		
o.B.	2	3120	hand rim, stainless steel 20", 8 knobs		
o.B.	2	3122	hand rim, stainless steel 22", 8 knobs		
o.B.	2	3124	hand rim, stainless steel 24", 8 knobs		
o.B.	2	3320	hand rim aluminium, 20", black, pvc coating		
o.B.	2	3322	hand rim aluminium, 22", black, pvc coating		
o.B.	2	3324	hand rim aluminium, 24", black, pvc coating		
o.B.	2	342405	handrim SURGE 24", 6 lugs		
o.B.	2	302078			
o.B.	2	302278			
o.B.	2	302478			

Handrim covers

pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
o.B.	2	322002	handrim cover silicone black ø442		04.67.06.20.00.40
o.B.	2	322202	handrim cover silicone black ø478		04.67.06.22.00.30
o.B.	2	322205	handrim cover silicone black ø453		04.67.06.22.00.20
o.B.	2	322208	handrim cover silicone black ø388		04.67.06.22.00.10
o.B.	2	322402	handrim cover silicone black ø532		04.67.06.24.00.30
o.B.	2	322404	handrim cover silicone black ø504		04.67.06.24.00.10
o.B.	2	322405	handrim cover silicone black ø514		04.67.06.24.00.20
o.B.	2	322603	handrim cover silicone black ø565		04.67.06.24.00.40
o.B.	2	322001	handrim cover profile wheel, Ergo 20"		
o.B.	2	322201	handrim cover profile wheel, Ergo 22"		
o.B.	2	322401	handrim cover profile wheel, Ergo 24"		

Spoke covers

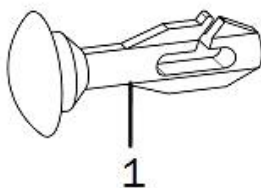


pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1	1	250001	clip for transparent spoke cover		25.00.99.01.00.10
o.B.	2	252200	spoke cover 22", Unicolor transparent		
o.B.	2	252400	spoke cover 24", Unicolor transparent		

Design-Dekor-Speichenschützer können Sie unter den folgenden Internetadressen wählen:


www.blueprince.de
www.speichen-schutz.de
www.wheelchairgadgets.com

Knee lever brake standard

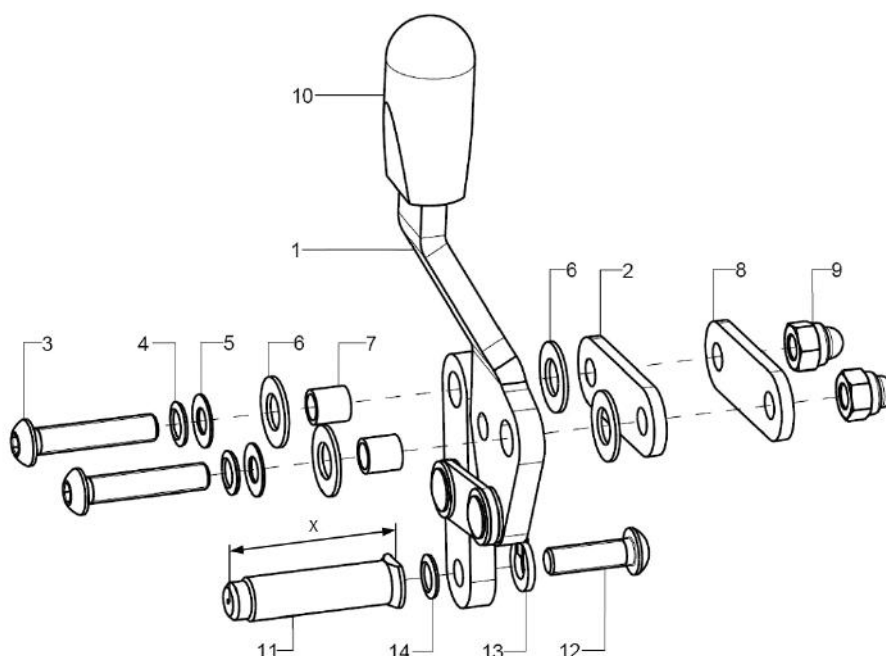


pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-14	1	352502	knee lever brake standard right complete		XXXXXXXXXXXX
o.B.	1	352501	knee lever brake standard left complete		XXXXXXXXXXXX
1	1	352504	brake pad standard pre-mounted right		35.25.01.02.02.10
o.B.	1	352503	brake pad standard pre-mounted left		35.25.01.02.01.10
2	2	352505	bracket 34 Bd 16x3x50		35.25.01.07.00.18
3-9	2	352508	mounting set knee lever brake		XXXXXXXXXXXX
3	2		lens head screw with hexagon socket ISO 7380 - M6x30		01.42.08.58.65.00
o.B.	2		lens head screw with hexagon socket ISO 7380 - M6x50		01.42.08.62.65.00
o.B.	2	352505	bracket 34 Bd 16x3x50		35.25.01.07.00.18
o.B.	2				
4	2		lock washer S6		02.99.10.60.01.10
5	2		fitted washer DIN 988 - 6x12x0,5 - A2		02.68.06.00.50.30
6	4		fender disc 8,4x16x1 PE		02.81.16.00.10.70
7	2				35.00.18.01.00.30
8	1	352505	bracket 34 Bd 16x3x50		35.25.01.07.00.18
9	2		hexagon cap nut low form with clamp DIN 986 - M6		02.33.00.60.11.20
10	2	350017	cap black 15x6,8		35.00.17.01.00.10
11-14	1	352518	brake bolt 40 mm left-hand thread incl. mounting set		35.25.01.18.00.10
o.B.	1	352506	brake bolt milled 40 mm incl. mounting set		35.25.01.15.00.10
altern.	1	352519	brake bolt 50 mm left-hand thread incl. mounting set		35.25.01.19.00.10
o.B.	1	352507	brake bolt milled 50 mm incl. mounting set		35.25.01.16.00.10
12	1		lens head screw with hexagon socket ISO 7380 , M6x20 left-hand thread		01.42.08.54.05.01
o.B.	1		lens head screw with hexagon socket ISO 7380 - M6x20		01.42.08.54.05.00
13	1		spring ring DIN 127- A6		02.85.30.60.05.10
14	1		lock washer S6		02.99.10.60.01.10

Knee lever brake pull-to-lock

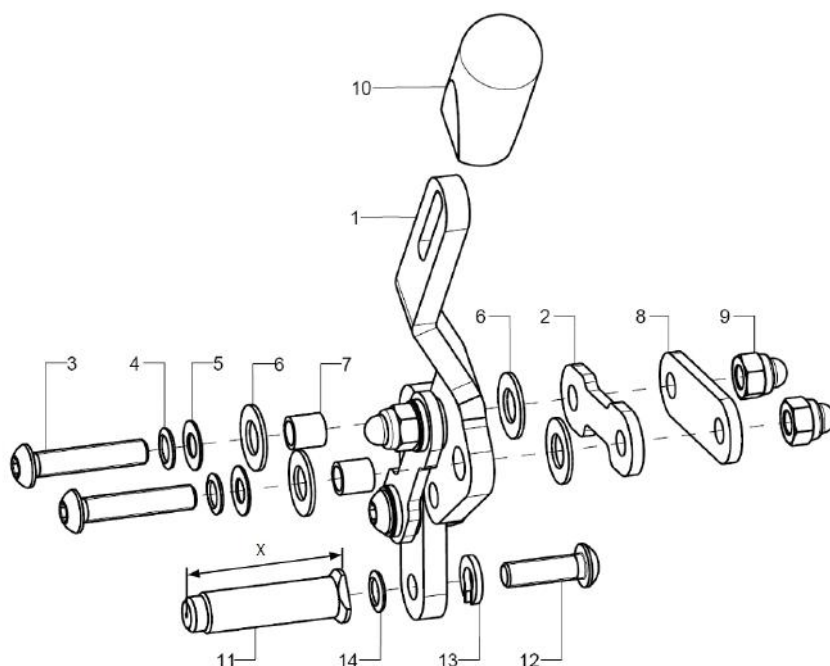


pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-14	1	352902	knee lever brake pull-to-lock right complete		XXXXXXXXXXXX
o.B.	1	352901	knee lever brake pull-to-lock left complete		XXXXXXXXXXXX
1	1	352904	brake pad pull-to-lock pre-mounted right		XXXXXXXXXXXX
o.B.	1	352903	brake pad pull-to-lock pre-mounted left		XXXXXXXXXXXX
2	2	352510	bracket 34, Bd 16x3x50 lockable		35.25.01.08.00.10
3-9	2	352508	mounting set knee lever brake		XXXXXXXXXXXX
3	2		lens head screw with hexagon socket ISO 7380 - M6x30		01.42.08.58.65.00
o.B.	2		lens head screw with hexagon socket ISO 7380 - M6x50		01.42.08.62.65.00
o.B.	2	352505	bracket 34 Bd 16x3x50		35.25.01.07.00.18
o.B.	2				
4	2		lock washer S6		02.99.10.60.01.10
5	2		fitted washer DIN 988 - 6x12x0,5 - A2		02.68.06.00.50.30
6	4		fender disc 8,4x16x1 PE		02.81.16.00.10.70
7	2				35.00.18.01.00.30
8	1	352505	bracket 34 Bd 16x3x50		35.25.01.07.00.18
9	2		hexagon cap nut low form with clamp DIN 986 - M6		02.33.00.60.11.20
10	2	350017	cap black 15x6,8		35.00.17.01.00.10
11-14	1	352518	brake bolt 40 mm left-hand thread incl. mounting set		35.25.01.18.00.10
o.B.	1	352506	brake bolt milled 40 mm incl. mounting set		35.25.01.15.00.10
altern.	1	352519	brake bolt 50 mm left-hand thread incl. mounting set		35.25.01.19.00.10
o.B.	1	352507	brake bolt milled 50 mm incl. mounting set		35.25.01.16.00.10
12	1		lens head screw with hexagon socket ISO 7380 , M6x20 left-hand thread		01.42.08.54.05.01
o.B.	1		lens head screw with hexagon socket ISO 7380 - M6x20		01.42.08.54.05.00
13	1		spring ring DIN 127- A6		02.85.30.60.05.10
14	1		lock washer S6		02.99.10.60.01.10

Knee lever brake lockable

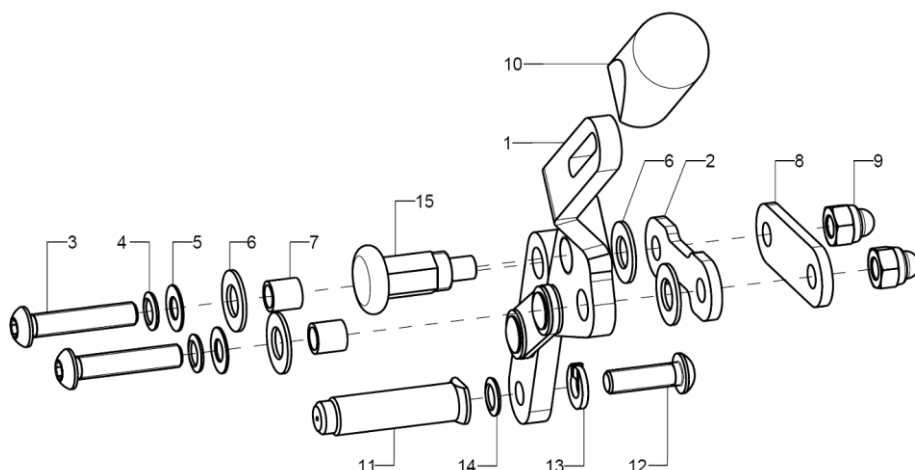

pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-15	1	351302	knee lever brake lockable complete right		XXXXXXXXXXXX
o.B.	1	351301	knee lever brake lockable complete left		XXXXXXXXXXXX
1	1	351304	brake pad lockable pre-assembled right		35.25.01.03.02.10
o.B.	1	351303	brake pad lockable pre-assembled left		35.25.01.03.01.10
2	2	352510	bracket 34, Bd 16x3x50 lockable		35.25.01.08.00.10
3-9	2	352508	mounting set knee lever brake		XXXXXXXXXXXX
3	2		lens head screw with hexagon socket ISO 7380 - M6x30		01.42.08.58.65.00
o.B.	2		lens head screw with hexagon socket ISO 7380 - M6x50		01.42.08.62.65.00
o.B.	2	352505	bracket 34 Bd 16x3x50		35.25.01.07.00.18
o.B.	2				
4	2		lock washer S6		02.99.10.60.01.10
5	2		fitted washer DIN 988 - 6x12x0,5 - A2		02.68.06.00.50.30
6	4		fender disc 8,4x16x1 PE		02.81.16.00.10.70
7	2				35.00.18.01.00.30
8	1	352505	bracket 34 Bd 16x3x50		35.25.01.07.00.18
9	2		hexagon cap nut low form with clamp DIN 986 - M6		02.33.00.60.11.20
10	2	350017	cap black 15x6,8		35.00.17.01.00.10
11-14	1	352518	brake bolt 40 mm left-hand thread incl. mounting set		35.25.01.18.00.10
o.B.	1	352506	brake bolt milled 40 mm incl. mounting set		35.25.01.15.00.10
altern.	1	352519	brake bolt 50 mm left-hand thread incl. mounting set		35.25.01.19.00.10
o.B.	1	352507	brake bolt milled 50 mm incl. mounting set		35.25.01.16.00.10
12	1		lens head screw with hexagon socket ISO 7380 , M6x20 left-hand thread		01.42.08.54.05.01
o.B.	1		lens head screw with hexagon socket ISO 7380 - M6x20		01.42.08.54.05.00
13	1		spring ring DIN 127- A6		02.85.30.60.05.10
14	1		lock washer S6		02.99.10.60.01.10
15	2	690101	latch ø6/M10x1x31 with lock		04.14.43.60.02.00

Knee lever brake for attendant

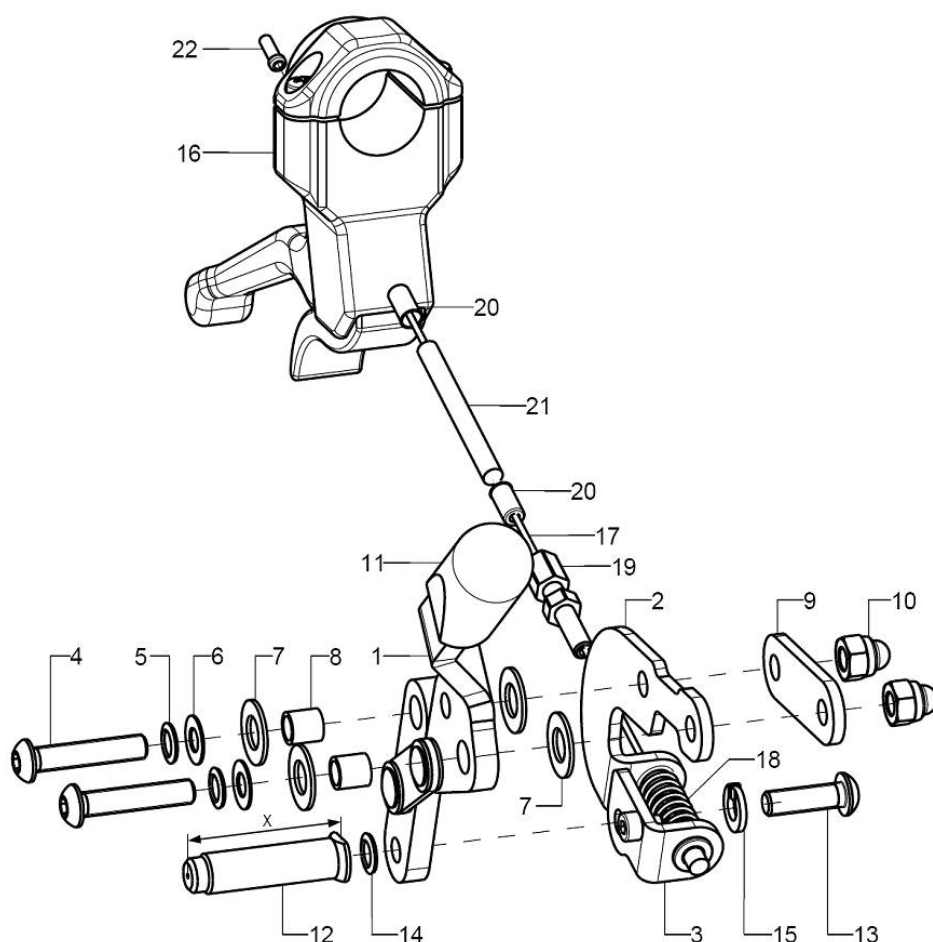

pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-22	1	352402	knee lever brake for attendant right incl. operating unit		XXXXXXXXXXXX
o.B.	1	352401	knee lever brake for attendant left incl. operating unit		XXXXXXXXXXXX
1	1	352504	brake pad standard pre-mounted right		35.25.01.02.02.10
o.B.	1	352503	brake pad standard pre-mounted left		35.25.01.02.01.10
2	1	35240102	holder brake bolt right		35.24.01.01.02.10
o.B.	1	35240101	holder brake bolt left		35.24.01.01.01.10
3	1	35240104	holder pull right		35.24.01.03.01.10
o.B.	1	35240103	holder pull left		35.24.01.03.02.10
4-10	2	352499	mounting set knee lever brake attendant		XXXXXXXXXXXX
4	2		lens head screw with hexagon socket ISO 7380 - M6x30		01.42.08.58.65.00
o.B.	2		lens head screw with hexagon socket ISO 7380 - M6x50		01.42.08.62.65.00
o.B.	2	352505	bracket 34 Bd 16x3x50		35.25.01.07.00.18
o.B.	2				
5	2		lock washer S6		02.99.10.60.01.10
6	2		fitted washer DIN 988 - 6x12x0,5 - A2		02.68.06.00.50.30
7	4		fender disc 8,4x16x1 PE		02.81.16.00.10.70
8	2				35.00.18.01.00.30
9	1	352505	bracket 34 Bd 16x3x50		35.25.01.07.00.18
10	2		hexagon cap nut low form with clamp DIN 986 - M6		02.33.00.60.11.20
11	2	350017	cap black 15x6,8		35.00.17.01.00.10
12-15	1	352518	brake bolt 40 mm left-hand thread incl. mounting set		35.25.01.18.00.10
o.B.	1	352506	brake bolt milled 40 mm incl. mounting set		35.25.01.15.00.10
altern.	1	352519	brake bolt 50 mm left-hand thread incl. mounting set		35.25.01.19.00.10




Spare parts list Vector

Brake

o.B.	1	352507	brake bolt milled 50 mm incl. mounting set	35.25.01.16.00.10
13	1		lens head screw with hexagon socket ISO 7380 , M6x20 left-hand thread	01.42.08.54.05.01
o.B.	1		lens head screw with hexagon socket ISO 7380 - M6x20	01.42.08.54.05.00
14	1		spring ring DIN 127- A6	02.85.30.60.05.10
15	1		lock washer S6	02.99.10.60.01.10
16-22	2	351810	hand brake lever for attendant brake complete	XXXXXXXXXXXX
16	1	352010	hand brake lever for attendant brake incl. mounting set	04.54.92.22.20.10
17	1	352012	inner Bowden cable1,5x2250	04.54.91.11.52.20
18	1	352405	press spring 1,1x9,4x44x3	04.21.09.40.44.00
19	1	352406	setting screw for Bowden cable M5x34 mm	04.54.01.40.50.21
20	2	352412	end bush metal ø5	04.54.91.50.10.41
21	1	352013	outer shell for Bowden cable black 150mm	04.54.91.21.53.00
22	1	352005	end cap for inner Bowden cable ø1-1,5 mm	04.54.91.50.10.40

One-hand brake operating side

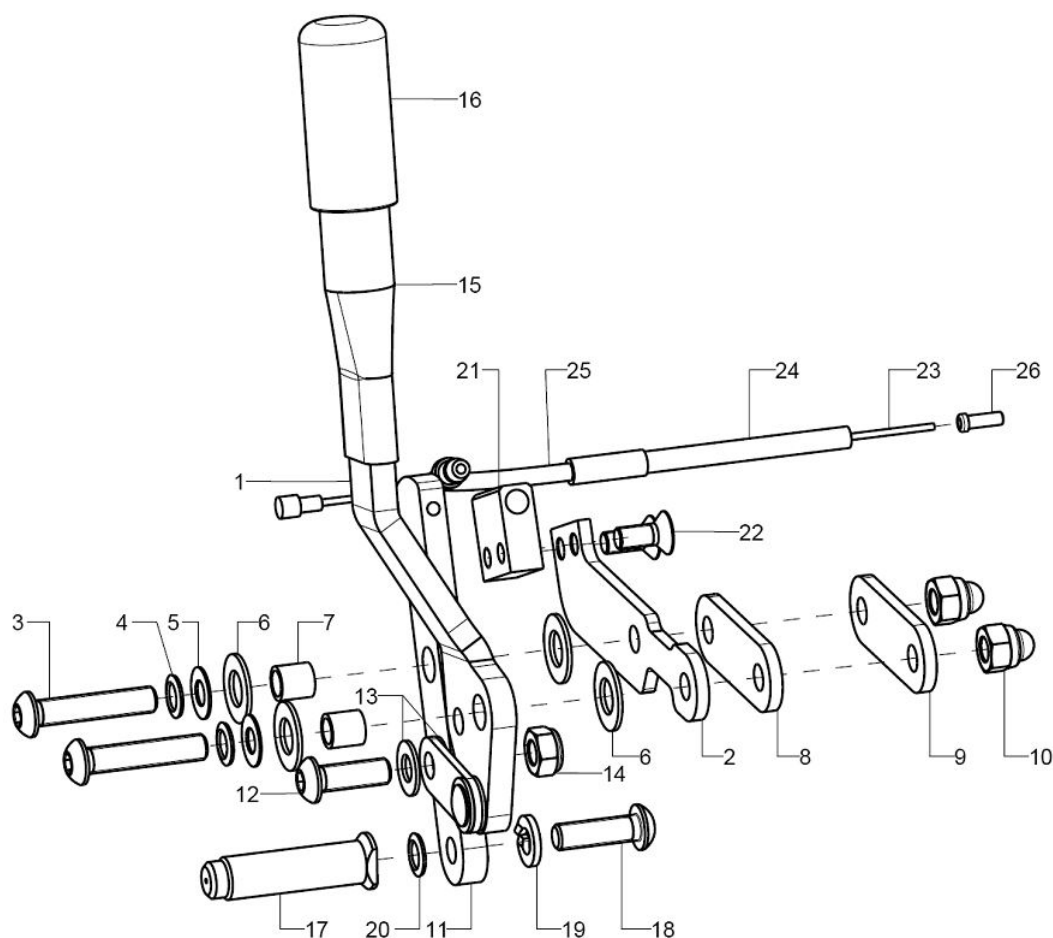

pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-49	1	350902	one-hand brake right, complete user side + non-operating side		XXXXXXXXXXXX
o.B.	1	350901	one-hand brake left, complete user side + non-operating side		XXXXXXXXXXXX
1-26	1	350904	one-hand brake, operating side right		XXXXXXXXXXXX
o.B.	1	350903	one-hand brake, operating side left		XXXXXXXXXXXX
1	1	350906	brake pad one-hand brake, pre-mounted right		XXXXXXXXXXXX
o.B.	1	350905	brake pad one-hand brake, pre-mounted left		XXXXXXXXXXXX
2	1	350910	holding plate operating side one-hand brake right		35.09.01.01.02.10
o.B.	1	350909	holding plate operating side one-hand brake left		35.09.01.01.01.10
3-10	1	350999	mounting set knee lever brake		XXXXXXXXXXXX
3	2		lens head screw with hexagon socket ISO 7380 - M6x30		01.42.08.58.65.00
o.B.	2		lens head screw with hexagon socket ISO 7380 - M6x50		01.42.08.62.65.00
o.B.	2	352505	bracket 34 Bd 16x3x50		35.25.01.07.00.18
o.B.	2				
4	2		lock washer S6		02.99.10.60.01.10
5	2		fitted washer DIN 988 - 6x12x0,5 - A2		02.68.06.00.50.30
6	4		fender disc 8,4x16x1 PE		02.81.16.00.10.70
7	2				35.00.18.01.00.30
8	1	352505	bracket 34 Bd 16x3x50		35.25.01.07.00.18
9	2	352505	bracket 34 Bd 16x3x50		35.25.01.07.00.18
10	2		hexagon cap nut low form with clamp DIN 986 - M6		02.33.00.60.11.20
11-14	1	350913	brake bolt holder one-hand brake incl. mounting set		35.09.01.03.00.10
12	1		lens head screw with hexagon socket ISO 7380 - M6x18		01.42.08.53.05.00




Spare parts list Vector

Brake

13	2		washer 6x12,7x0,9	02.83.12.01.00.50
14	1		hexagon nut self-locking low form DIN 985 - M6	02.32.00.60.01.20
15	1			
16	1	351011	PVC cap one-hand brake	04.63.00.16.40.40
17-20	1	352518	brake bolt 40 mm left-hand thread incl. mounting set	35.25.01.18.00.10
o.B.	1	352506	brake bolt milled 40 mm incl. mounting set	35.25.01.15.00.10
altern.	1	352519	brake bolt 50 mm left-hand thread incl. mounting set	35.25.01.19.00.10
o.B.	1	352507	brake bolt milled 50 mm incl. mounting set	35.25.01.16.00.10
18	1		lens head screw with hexagon socket ISO 7380 , M6x20 left-hand thread	01.42.08.54.05.01
o.B.	1		lens head screw with hexagon socket ISO 7380 - M6x20	01.42.08.54.05.00
19	1		spring ring DIN 127- A6	02.85.30.60.05.10
20	1		lock washer S6	02.99.10.60.01.10
21-22	1	350914	holder one-hand brake operating side incl. mounting screws	35.09.01.02.00.10
22	2		counter sunk screw with hexagon socket DIN 7991 M5x12	01.65.07.89.05.00
23-26	1			XXXXXXXXXXXX
23	1	352012	inner Bowden cable1,5x2250	04.54.91.11.52.20
24	1	350915	outer shell for Bowden cable	04.54.91.21.53.00
25	2	350916	braket with bush	35.09.03.01.00.10
26	1	352005	end cap for inner Bowden cable ø1-1,5 mm	04.54.91.50.10.40

One-hand brake non-operating side

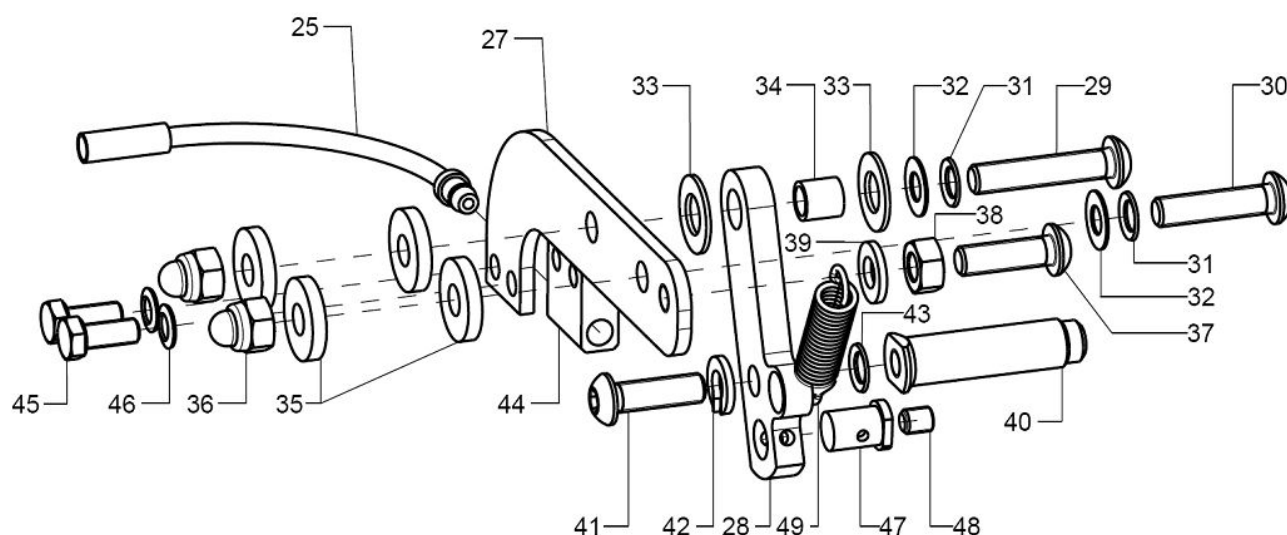

pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
27-49	1	350912	one-hand brake non-operating side complete		XXXXXXXXXXXX
27	1	350920	holding plate one-hand brake		35.09.02.01.00.10
28	1	350921	brake bolt holder one-hand brake		35.09.02.02.00.10
29-39	1	350998	mounting set one-hand brake		XXXXXXXXXXXX
29	1		lens head screw with hexagon socket ISO 7380 - M6x30		01.42.08.58.65.00
o.B.	1		lens head screw with hexagon socket ISO 7380 - M6x50		01.42.08.62.65.00
30	1		lens head screw with hexagon socket ISO 7380 - M6x25		01.42.08.56.65.00
o.B.	1		lens head screw with hexagon socket ISO 7380 - M6x45		01.42.08.61.65.00
o.B.	1	352505	bracket 34 Bd 16x3x50		35.25.01.07.00.18
o.B.	2				
31	2		lock washer S6		02.99.10.60.01.10
32	2		fitted washer DIN 988 - 6x12x0,5 - A2		02.68.06.00.50.30
33	2		fender disc 8,4x16x1 PE		02.81.16.00.10.70
34	1				35.00.18.01.00.30
35	4		washer DIN 7349 - 6,4		02.77.06.53.00.20
36	2		hexagon cap nut low form with clamp DIN 986 - M6		02.33.00.60.11.20
37	1		lens head screw with hexagon socket ISO 7380 - M6x20		01.42.08.54.05.00
38	1		hexagon nut DIN 934 - M10		02.19.01.00.01.20
39	1		washer DIN 125 - A 6,4		02.61.06.41.41.20
40-43	1	352518	brake bolt 40 mm left-hand thread incl. mounting set		35.25.01.18.00.10
o.B.	1	352506	brake bolt milled 40 mm incl. mounting set		35.25.01.15.00.10
altern.	1	352519	brake bolt 50 mm left-hand thread incl. mounting set		35.25.01.19.00.10
o.B.	1	352507	brake bolt milled 50 mm incl. mounting set		35.25.01.16.00.10
41	1		lens head screw with hexagon socket ISO 7380 , M6x20 left-hand thread		01.42.08.54.05.01
o.B.	1		lens head screw with hexagon socket ISO 7380 - M6x20		01.42.08.54.05.00
42	1		spring ring DIN 127- A6		02.85.30.60.05.10




Spare parts list Vector

Brake

43	1		lock washer S6	02.99.10.60.01.10
44-46	1	350922	holder one-hand brake non-operating side incl. mounting screws	35.09.01.02.00.10
45	2		hexagon screw DIN 933 - M5x12	01.04.07.89.02.40
46	2		lock washer S5	02.99.10.50.01.10
47-48	1	350923	clamp unit incl. setscrew	Rolko_Art.-Nr.
48	1		setscrew with hexagon socket DIN 913, M5x6	01.67.07.86.61.20
49	1	350924	pull spring 1,2x9x40	04.22.00.10.40.00

Bowden cable brake pressure piece

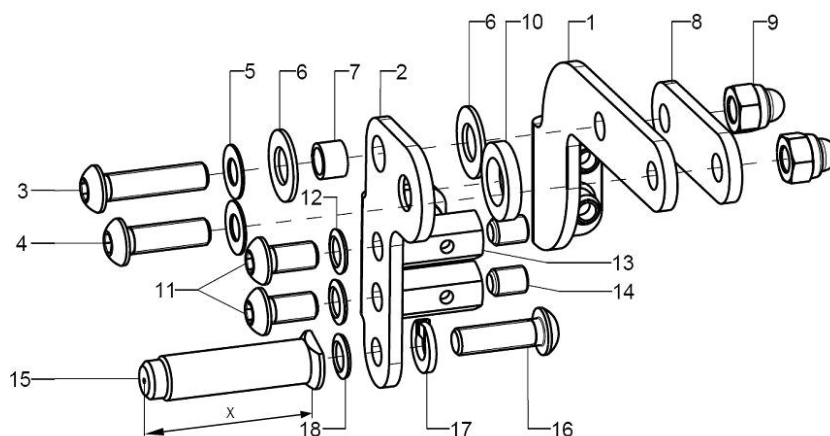


pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-32	1	3531	cable brake complete		XXXXXXXXXXXX
1	1	35310604	base unit combination right		35.31.06.04.02.69
o.B.	1	35310205	base part right		35.31.02.04.02.69
o.B.	1	35310603	base unit combination left		35.31.06.04.01.69
o.B.	1	35310204	base part left		35.31.02.04.01.69
2	1	35310606	pressing part right		35.31.06.03.02.69
o.B.	1	35310202	pressing part right		35.31.02.01.02.69
o.B.	1	35310605	pressing part left		35.31.06.03.01.69
o.B.	1	35310201	pressing part left		35.31.02.01.01.69
3-14	2	353199	mounting set for cable brake		XXXXXXXXXXXX
3	1		lens head screw with hexagon socket ISO 7380 - M6x25		01.42.08.56.65.00
o.B.	1		lens head screw with hexagon socket ISO 7380 - M6x40		01.42.08.60.65.00
4	1		lens head screw with hexagon socket ISO 7380 - M6x20		01.42.08.54.05.00
o.B.	1				01.42.08.59.62.60
o.B.	1	352505	bracket 34 Bd 16x3x50		35.25.01.07.00.18
o.B.	2				
5	2		fitted washer DIN 988 - 6x12x0,5 - A2		02.68.06.00.50.30
6	2		fender disc 8,4x16x1 PE		02.81.16.00.10.70
7	1				35.31.02.03.00.69
8	1	352505	bracket 34 Bd 16x3x50		35.25.01.07.00.18
9	2		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20
10	1				19.06.02.07.00.24
11			lens head screw with hexagon socket ISO 7380 - M6x12		01.42.08.49.65.00
12			lock washer S6		02.99.10.60.01.10
13					35.31.02.06.00.69
14			setscrew with hexagon socket DIN 913, M6x8		01.67.08.47.61.20
15-18	1	352518	brake bolt 40 mm left-hand thread incl. mounting set		35.25.01.18.00.10
o.B.	1	352506	brake bolt milled 40 mm incl. mounting set		35.25.01.15.00.10
altern.	1	352519	brake bolt 50 mm left-hand thread incl. mounting set		35.25.01.19.00.10
o.B.	1	352507	brake bolt milled 50 mm incl. mounting set		35.25.01.16.00.10
16	1		lens head screw with hexagon socket ISO 7380 , M6x20 left-hand thread		01.42.08.54.05.01
o.B.	1		lens head screw with hexagon socket ISO 7380 - M6x20		01.42.08.54.05.00
17	1		spring ring DIN 127- A6		02.85.30.60.05.10
18	1		lock washer S6		02.99.10.60.01.10


Bowden cable brake control/non-control sides

Seilzugbremse

Benutzerbedienteil

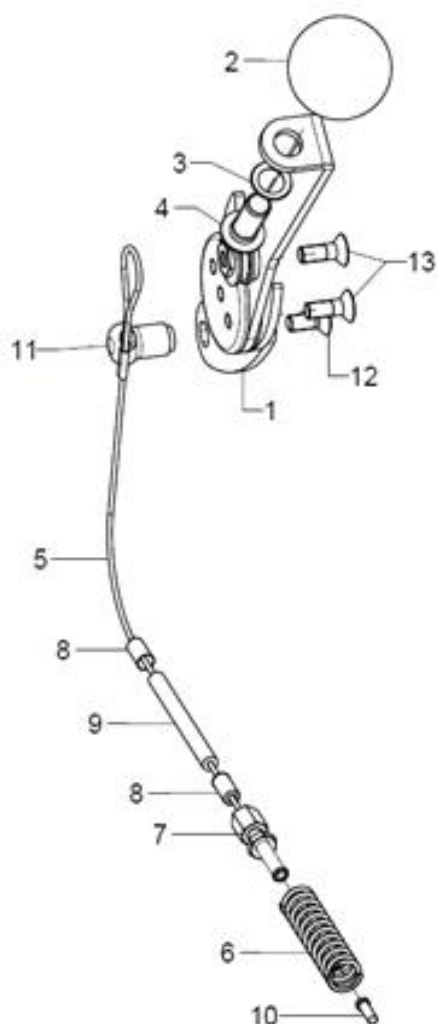


Begleiterbedienteil

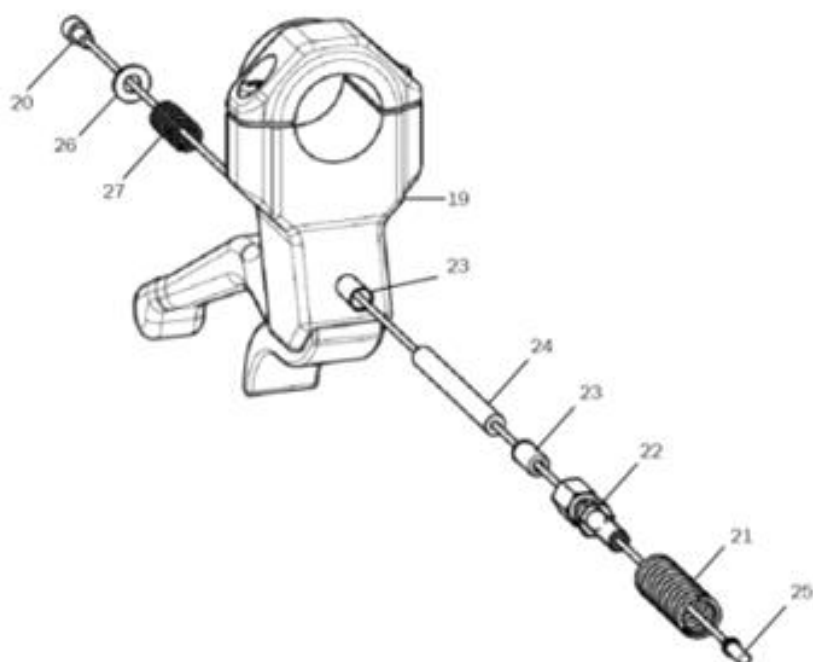




pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-13	2	353103	user operating unit complete+ cable brake complete		XXXXXXXXXXXX
1	1	353111	operating unit cable brake Jump/Vector pre-mounted right		35.31.01.99.02.69
o.B.	1	353112	operating unit cable brake Jump/Vector pre-mounted left		35.31.01.99.01.74
2-4	1	35310109	ball knob Ø32/M8, form C, incl. mounting set		04.06.13.20.80.50
3	1		lock washer S8		02.99.10.80.01.10
4	1		lens head screw with hexagon socket ISO 7380 - M8x20		01.42.09.74.65.00
5-10+13	2	353105	cable brake user operating unit complete		XXXXXXXXXXXX
5	1	35310301	bowden cable with loop 90 cm		35.31.03.01.00.69
6	1	352405	press spring 1,1x9,4x44x3		04.21.09.40.44.00
7	1	352406	setting screw for Bowden cable M5x34 mm		04.54.01.40.50.21
8	2	352412	end bush metal Ø5		04.54.91.50.10.41
9	1	35310303	outer cover for Bowden cable black, 38 cm long		04.54.91.21.53.00
10	1	352005	end cap for inner Bowden cable Ø1-1,5 mm		04.54.91.50.10.40
11	1				35.31.03.03.00.69
12	1		counter sunk screw with hexagon socket DIN 7991 M5x12		01.65.07.89.05.00
13	2		counter sunk screw with hexagon socket DIN 7991 M5x14		01.65.07.90.05.00
19-27	2	353110	attendant operating unit		XXXXXXXXXXXX
19	1	352010	hand brake lever for attendant brake incl. mounting set		04.54.92.22.20.10
20	1	352012	inner Bowden cable1,5x2250		04.54.91.11.52.20
21	1	352405	press spring 1,1x9,4x44x3		04.21.09.40.44.00
22	1	352406	setting screw for Bowden cable M5x34 mm		04.54.01.40.50.21
23	2	352412	end bush metal Ø5		04.54.91.50.10.41
24	1	352013	outer shell for Bowden cable black 150mm		04.54.91.21.53.00
25	1	352005	end cap for inner Bowden cable Ø1-1,5 mm		04.54.91.50.10.40
26	1		washer DIN 125 - A 4,3		02.61.04.30.91.20
27	1		pressure spring 0,66x6,7x15		04.21.06.60.15.00

Brake lever for drum brake

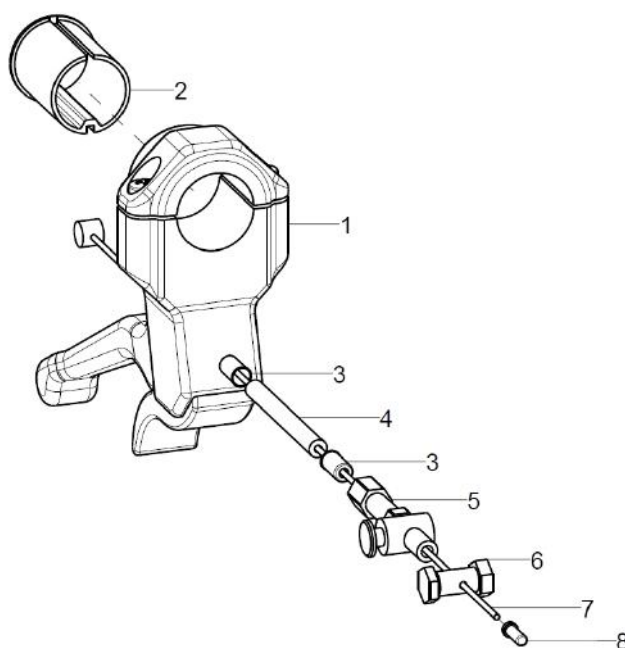


pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-8	2	352016	hand brake lever complete		XXXXXXXXXXXX
1+2	1	352010	hand brake lever for attendant brake incl. mounting set		04.54.92.22.20.10
2	1	352011			35.20.11.01.00.28
3-8	2	352006			
3	2	352412	end bush metal ø5		04.54.91.50.10.41
4	1	352013	outer shell for Bowden cable black 150mm		04.54.91.21.53.00
5	1	352007	set screw for Bowden cable M6x40 mm		04.54.01.40.60.20
6	1	352009			04.54.11.30.00.20
7	1	352012	inner Bowden cable 1,5x2250		04.54.91.11.52.20
8	1	352005	end cap for inner Bowden cable ø1-1,5 mm		04.54.91.50.10.40

Brake lever extensions

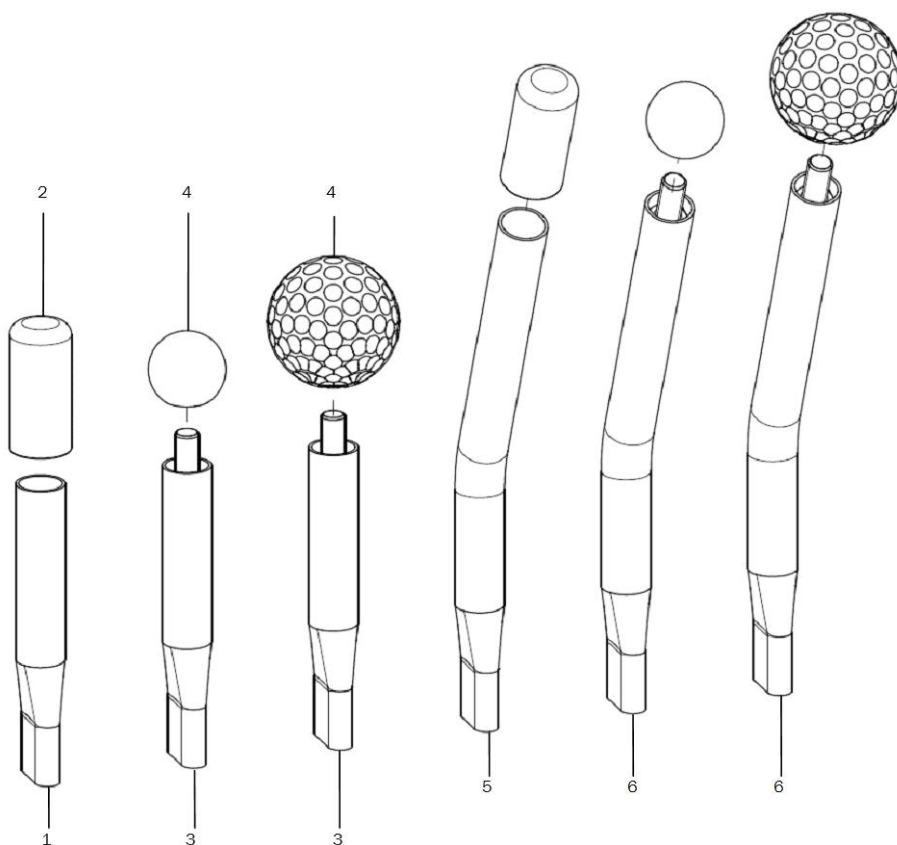


pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-2	2	3510	brake lever extension with cap		35.01.03.02.00.10
2	2	351011	PVC cap one-hand brake		04.63.00.16.40.40
3-4	2	3512	brake lever extension 100 for ball knob		35.14.03.01.00.30
4	2	351016	ball knob		04.06.12.50.80.40
altern.	2	351211	golf ball		04.06.14.20.80.00
5+2	2	3515	brake lever extension 23° with cap		35.15.03.01.00.10
6+4	2	3516	brake lever extension 23° for ball knob		35.16.03.01.00.10

Caddy

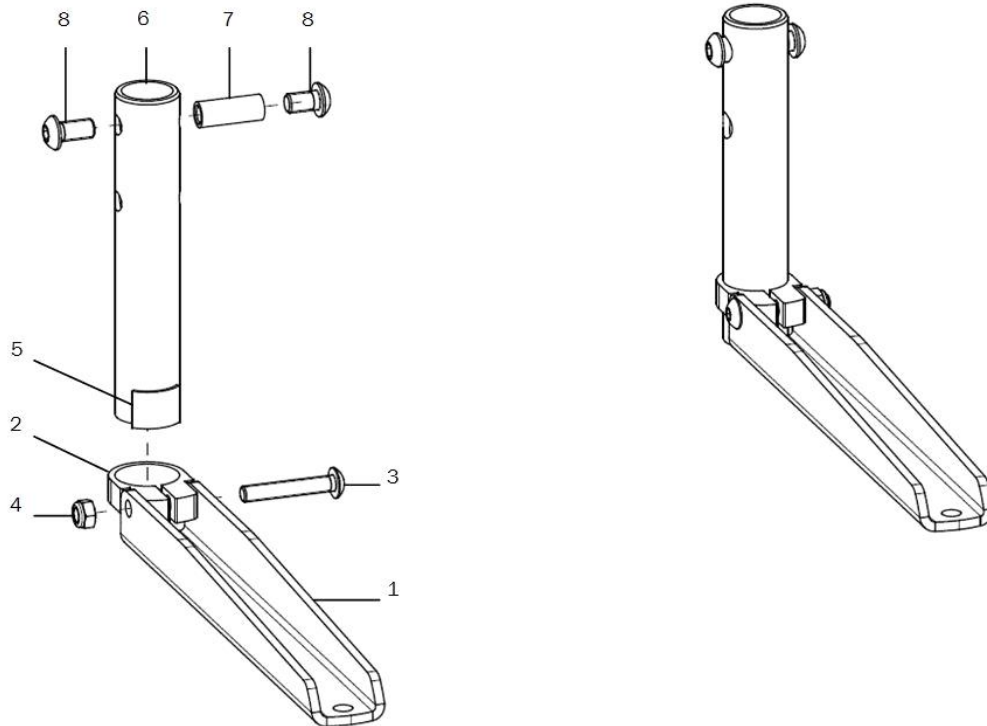

pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-8	2	910112	baggage holder for leg support addition, inner mounting complete		XXXXXXXXXXXX
1-5	2	910110	baggage holder for leg support addition, outer mounting complete		XXXXXXXXXXXX
1	1	910101	attachment for baggage holder		91.01.01.01.00.74
2	1	910102	clamp ø29mm		91.01.01.02.00.74
3	1		lens head screw with hexagon socket ISO 7380 - M5x30		01.42.07.98.65.00
4	1		hexagon nut self-locking low form DIN 985 - M5		02.32.00.50.01.20
5	1		Velcro strap 20 mm, black		00.42.60.02.50.10
6-8	1	910103	rod adapter incl mounting set		91.01.01.03.00.74
7	1		bushing M6x20,5		40.39.01.04.00.69
8	2		lens head screw with hexagon socket ISO 7380 - M6x10		01.42.08.48.65.00

Water bottle

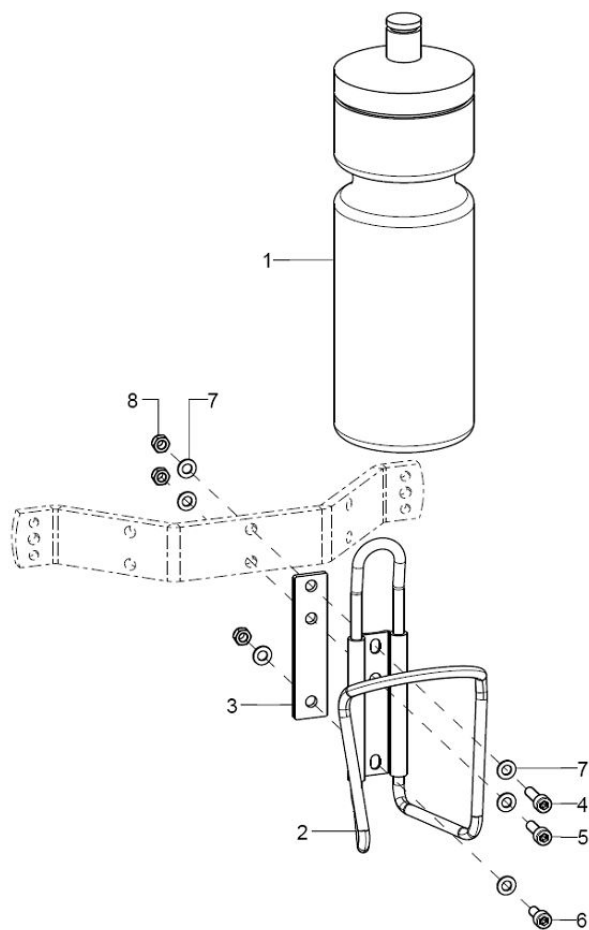

pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-8	1	790101	water bottle complete incl. holder		XXXXXXXXXXXX
1	1	790102	water bottle silver 0,75L		79.01.01.02.00.10
2-8	1	790103	water bottle holder complete		XXXXXXXXXXXX
2	1				
3	1				
4	1		cylinder screw DIN 912 M5x18		01.22.07.93.72.40
5	1		cylinder head screw DIN 912 M5x16		01.22.07.92.72.40
6	1		cylinder screw DIN 912 M5x12		01.22.07.89.72.40
7	6		washer DIN 125 - A 5,3		02.61.05.31.01.20
8	3		hexagon nut self-locking low form DIN 985 - M5		02.32.00.50.01.20

Hip belt with Velcro and synthetic lock



pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1	1	7800	lap belt small		04.56.90.01.00.20
o.B.	1	7801	lap belt, medium cushion		04.56.90.01.00.10

Hip belt with metal lock



pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1	1	7805	lap belt by Bodypoint with metal lock and mounting clamp		04.56.91.01.00.10

Butterfly harness



pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1	1	7810	butterfly harness		