

Spare Parts Catalogue

Version: 2016/05/26a EN





Yes, you can.®

General Information

For any order of spare parts the following information is required:

- Product type*
- Product name & model*
- Serial Number*
- Part-Number*
- Part-Description*

Exploded view drawings and code numbers are not contractual and subject to modification, in relation to Invacare® continuous product improvement policy to meet end users needs.

The reproduction or use of this work, or of any substantial part of this work, in any form or by any electronic, mechanical or other means, including photocopying, scanning and recording, and in any information storage and retrieval system, including electronic transmission, is forbidden without the prior written permission of Invacare® International Sarl.



Yes, you can.®

TABLE OF CONTENTS

Chassis

Chassis	7
Chassis, Azalea Base	8
Chassis extender	10
Backrest + seat tilt, Azalea	11
Backrest + seat tilt, electric, Azalea	12
Backrest + seat tilt, electric, Azalea Max	13
Backrest + seat tilt, electric (=>2015.08)	14
Castor attachment, fixed	15
Kit, Azalea Tall	16

Seat

Seat cushion, Flo-shape	17
Seat cushion, Flo-shape =>2012-04-01	18
Seat cushion, Vicair multifunctional	19
Seat cushion, Flo-tech contour Visco (only for France)	20
Seat cushion, Mistral2	21
Seat cushion, Azalea Max	22
Incontinents cover	23
Sliding seat	24

Backrest

Backrest "Flex3"	25
Backrest "Flex3", tension adj.	26
Backrest, Azalea minor	27
Backrest "Flex2", fixed (->2013)	28
Backrest "Flex2", adj. (->2013)	29
Backrest, Laguna (France only)	30
Backrest, Azalea Max	31
Cushions, Flex3 backrest	32
Cushions, Flex2 backrest	34
Cushions, Flex2 backrest =>2012-04-01	36
Curvature cushion	38
Backrest Matrix	39
Backrest Matrix, Azalea Base	41

Legrests

Legrest attachment	42
Legrest 80° Alu2003, Fixed	43
Legrest 90° Alu2003, Fixed	44
Legrest AA, ALU2008	45
Legrest AA, Azalea Max	46
Amputee legrest	47
Amputee legrest, Azalea Max	48

Footrests

Footrest, fixed, Alu2003	49
Footrest, AA, Alu2003	50
Footrest 90°, Alu2003	52



Yes, you can.®

TABLE OF CONTENTS

Footboard, Azalea Max	53
Central footrest	54
Calf strap, Alu2003 legrest	55
Calf strap, padded	56
Heel strap	57
Extender, footplate	58
Footrest one piece	59
Footplate cover	60
Footbox	61

Siderests

Armrest, Ducktail	62
Armrest Low, Ducktail	63
Armrest, height adj.	64
Armrest, padded	65
Armrest Max, reinforced	66
Armrest, hemiplegic	67
Pad, siderest	68
Armrest attachment	69
Armrest attachment, Azalea Max	70

Front Wheels

Castor wheels	71
---------------------	----

Rear Wheels

Wheels 16"	72
Wheels 16" (=>2010.11)	73
Wheels 24" Basic	74
Wheels 24" Basic DB	75
Wheel 24-22-20"	76
Handrim Stainless Cpl, 24" Wheel, Std2010	78
Wheel 24-22" DB	79
Handrim Stainless Cpl, 24" Wheel, Std2010	78

Brakes

Brakes	81
Brake, Azalea Minor/Azalea Base sw34	82
Brake, Azalea Max	83
Attachment Ri, Brake Extended	84
Brake, extended, Azalea Max	85
Attachment Ri, Brake Extended	84
Brake, Azalea Tall	86
Brake attachment	87
Drumbrake, assistant manoeuvred	88
Drumbrake, assistant manoeuvred (=>11.2010)	89
Brake, user manoeuvred, 16" wheel	90

Safety Options

Pelvic belt	91
Pelvic belt, Azalea Base (04.2011 =>)	92



Yes, you can.®

TABLE OF CONTENTS

Pelvic belt, Max	93
X-harness, Azalea Minor	94
Anti-tip device	95
Reflector	96
Attachment, X-harness, Flex3	97


Seating Options

Headrest	98
Neckrest	99
Cheek support	100
Pommel	101
Trunk supports, Flex3 backrest	102
Trunk supports, Flex3 tension adj. backrest	103
Trunk supports, Flex2	104
Trunk supports, Azalea MAX	105
Trunk supports, Azalea Minor	106
Trunk support, Laguna2	107
Attachment, Positioning Belt	108
Head-/Neckrest pole	109

Others Options & Accessories


Table tray	110
Table tray, Azalea Max	111
Table tray with armpad	112
Table tray "Swingaway"	113
Table tray "Swingaway", Azalea Minor	114
Table tray "Swingaway"	113
Table tray, Azalea Max	111
Pushbar adj. Azalea Minor	115
Pushbar adj. Azalea	116
Push bar, Flex2	117
Push handles, connected, Flex2	118
Push handles, Azalea Max	119
Crutch holder, Flex3 backrest	120
Crutch holder, Flex2 backrest	121
Wash sealing kit	122
Tilt scale	123
Clinic kit	124
Castor lock, 16" + 22" wheels	125
Dripstand	126

Chassis

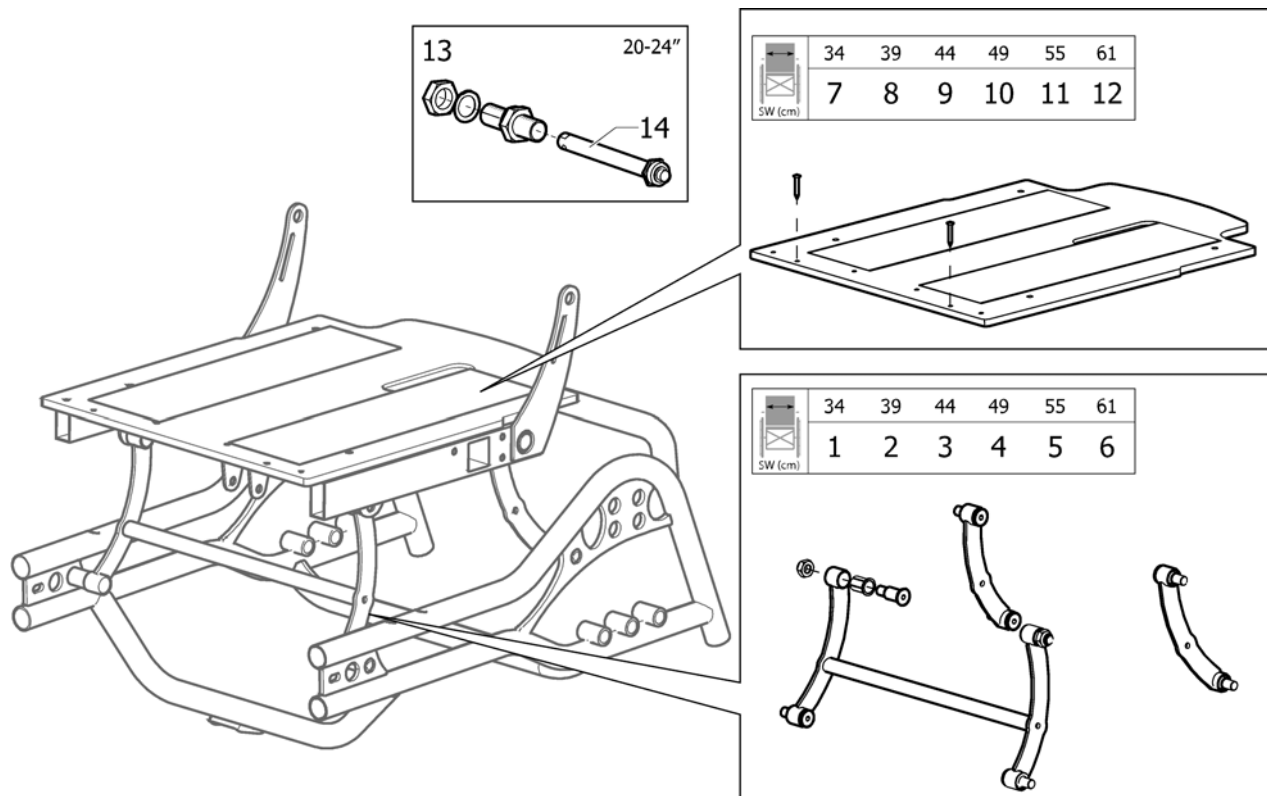

Pos.	Part number	Description	Valid from → until	Qty
1	1508928	Link Arm Kit, Seat Frame 34, Azalea		1
2	1451810	Link Arm Kit, Seat Frame 39, Azalea		1
3	1451811	Link Arm Kit, Seat Frame 44, Azalea		1
4	1451839	Link Arm Kit, Seat Frame 49, Azalea		1
5	1535595	Link Arm Kit, Seat Frame 55, Azalea		1
6	1535596	Link Arm Kit, Seat Frame 61, Azalea		1
7	1513123	Seat Plate 34 Cpl.		1
8	1508135	Seat Plate 39 Cpl.		1
9	1508136	Seat Plate 44 Cpl.		1
10	1508137	Seat Plate 49 Cpl.		1
11	1525083	Seat Plate 55 Cpl.		1
12	1525082	Seat Plate 61 Cpl.		1
13	5750166	QR axle kit (when serial number start by 02)		1
14	5750165	QR Axle 12mm		1

Chassis, Azalea Base

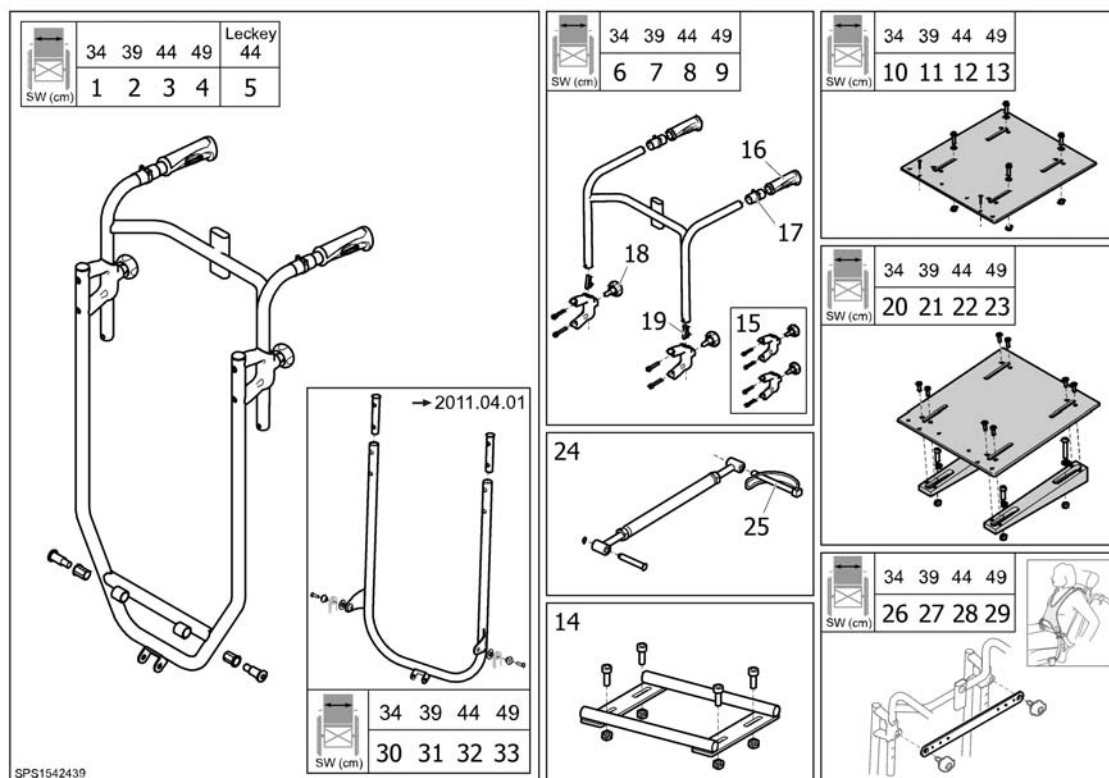


Pos.	Part number	Description	Valid from → until	Qty
1	1542429	BACKREST 34 CPL		1
2	1542430	BACKREST 39 CPL		1
3	1542431	BACKREST 44 CPL		1
4	1542433	BACKREST 49 CPL		1
5	1542432	BACKREST 44 CPL, LECKEY		1
6	1512900	Bow, Head & Neckrest, Sw 33/34		1
7	1435750	Bow, Head & Neckrest, Sw 38 (355)		1
8	1435751	Bow, Head & Neckrest, Sw 43 (405)		1
9	1435752	Bow, Head & Neckrest, Sw 48 (455)		1
10	1539677	SEATPLATE 34 CPL		1
11	1539679	SEATPLATE 39 CPL		1
12	1539681	SEATPLATE 44 CPL		1
13	1539683	SEATPLATE 49 CPL		1
14	1508908	H-ATTACHMENT, SEAT		1
15	1512904	Attachment, Pushhandles, Pair		1
16	SP1429301	HANDLE 20MM, PAIR		1
17	1542447	ADAPTER, HANDLE		1
18	1434841	Handwheel M10-22,5		1
19	1560734	Spring pin locked armrest		1
20	1542434	SEATPLATE 34 CPL, WITH WEDGES		1
21	1542435	SEATPLATE 39 CPL, WITH WEDGES		1
22	1542436	SEATPLATE 44 CPL, WITH WEDGES		1
23	1542437	SEATPLATE 49 CPL, WITH WEDGES		1
24	1512895	KIT, RIGGING SCREW		1
25	1439936	Pin with Loop 8X60		1
26	1542445-0509	ATTACHMENT CPL., X-HARNESS 34		1
27	1539655-0509	ATTACHMENT CPL., X-HARNESS 39		1
28	1539662-0509	ATTACHMENT CPL., X-HARNESS 44		1
29	1542446-0509	ATTACHMENT CPL., X-HARNESS 49		1
30	1512896	KIT, BACKREST BOW 34		1
31	1512897	KIT, BACKREST BOW 39		1

Chassis, Azalea Base


Pos.	Part number	Description	Valid from → until	Qty
32	1512898	KIT, BACKREST BOW 44		1
33	1512899	KIT, BACKREST BOW 49		1

Chassis extender

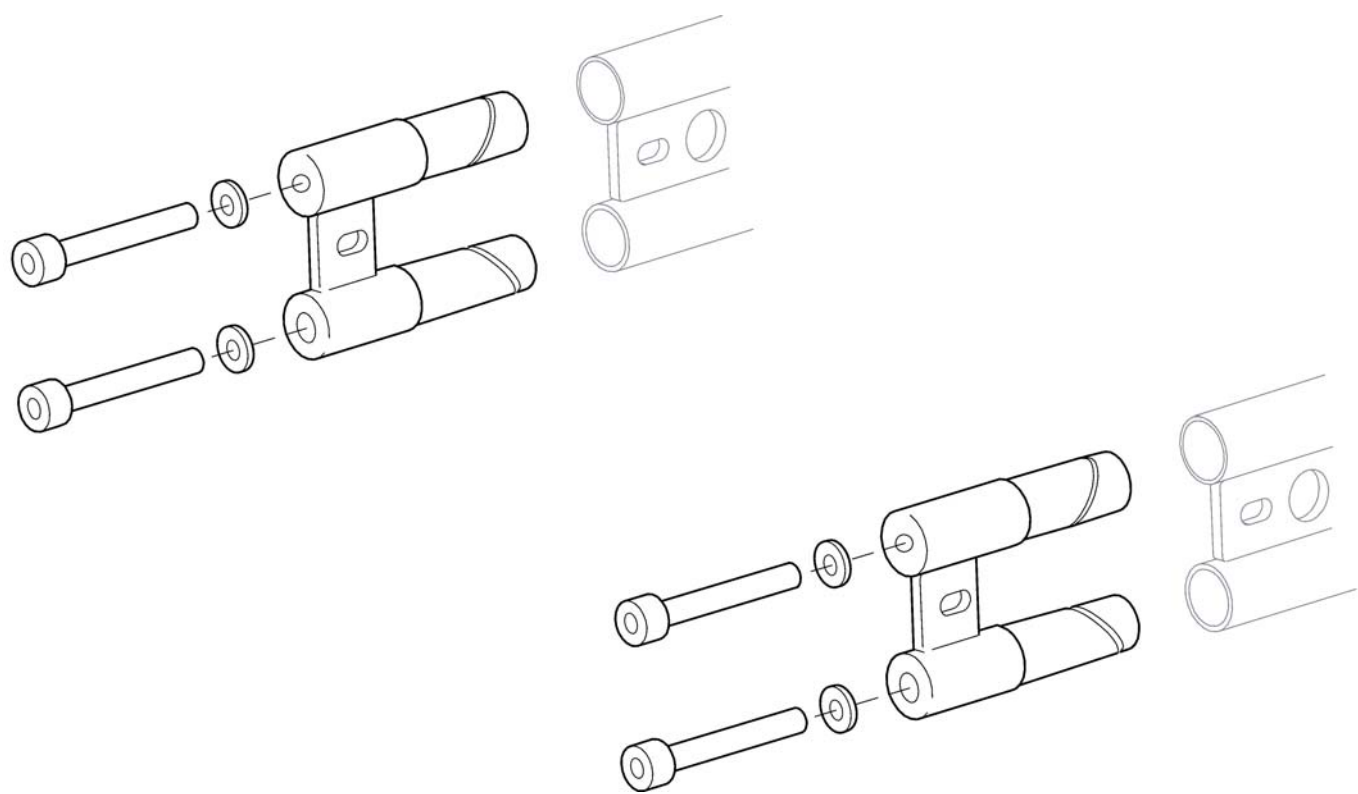


Pos.	Part number	Description	Valid from → until	Qty
	SP1484535	Chassis extender		1

Backrest + seat tilt, Azalea

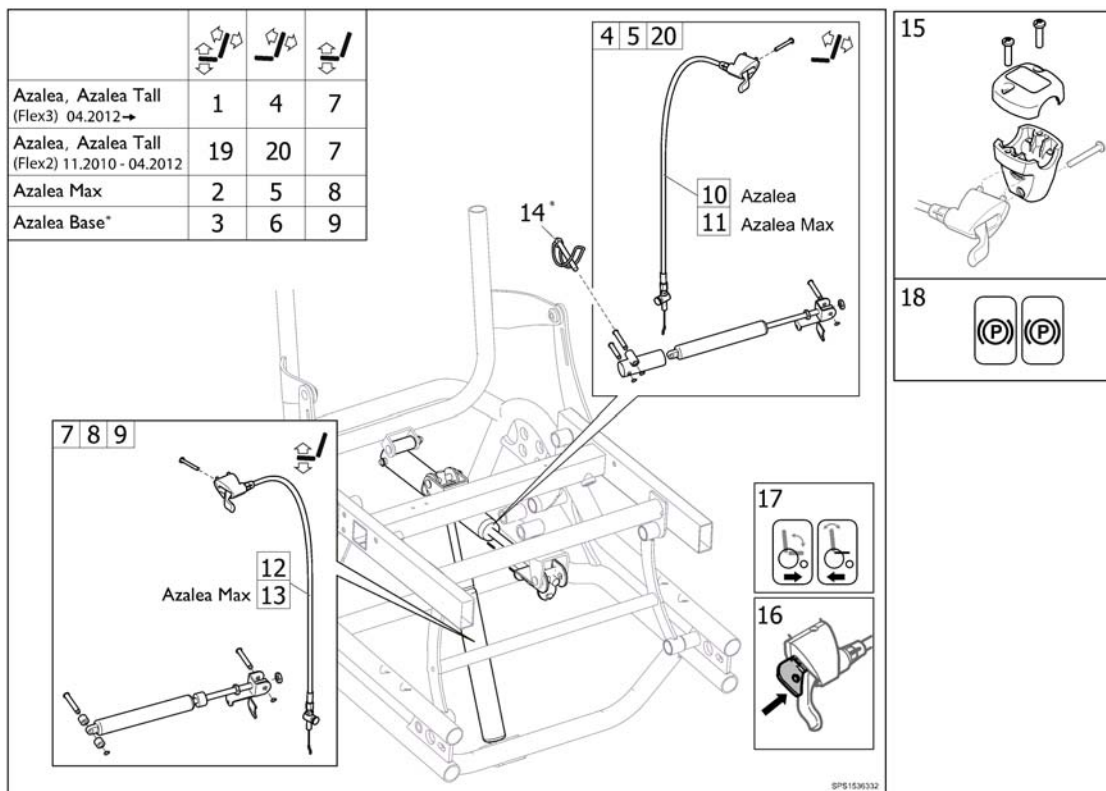

Pos.	Part number	Description	Valid from → until	Qty
1	1559684	KIT, TILT BACKREST + SEAT, ASSISTANT MAN., AZALEA (FLEX3)		1
2	1536336	KIT, TILT BACKREST + SEAT, ASSISTANT MAN., AZALEA MAX		1
3	1542606	KIT, TILT BACKREST + SEAT, ASSISTANT MAN., AZALEA BASE		1
4	1559495	MANOUVERING KIT BACKREST CPL. (FLEX3)		1
5	1536338	MANOUVERING KIT BACKREST CPL		1
6	1539661	MANOUVERING KIT BACKREST CPL		1
7	1536339	MANOUVERING KIT SEAT CPL		1
8	1536341	MANOUVERING KIT SEAT CPL		1
9	1536843	MANOUVERING KIT SEAT, BASE CPL		1
10	1536328	CABLE ASSY 1190, LH, TILT, INCL. SCREW		1
11	1536330	CABLE ASSY 1363, LH, TILT, INCL. SCREW		1
12	1536325	CABLE ASSY 1190, RH, TILT, INCL. SCREW		1
13	1536327	CABLE ASSY 1363, RH, TILT, INCL. SCREW		1
14	1439936	Pin with Loop 8X60		1
15	1527287	Housing Cpl, Tilt		1
16	1527435	LOCKING DEVICE, MANOUVERING KIT		1
17	1536373	DECAL, CABLE ASSEMBLY, TILT		1
18	1536764	DECAL, CABLE ASSEMBLY, BRAKE		1
19	1536332	KIT, TILT BACKREST + SEAT, ASSISTANT MAN., AZALEA		1
20	1536337	MANOUVERING KIT BACKREST CPL		1


Backrest + seat tilt, electric, Azalea

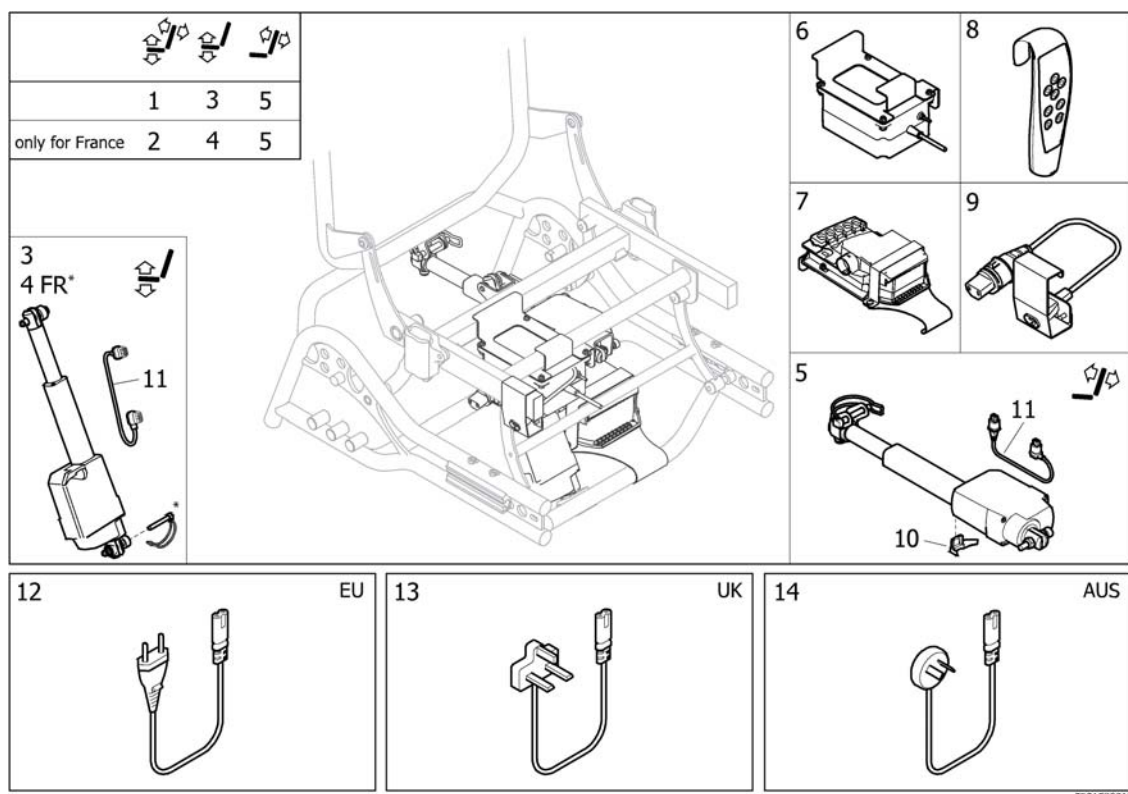

SPS1583212


Pos.	Part number	Description	Valid from → until	Qty
1	SP1583212	Kit, power tilt, Azalea (powercable not included)		1
2	SP1583400	Kit, power tilt, Azalea (France), powercable not included		1
3	SP1583177	Kit, seat tilt cpl.		1
4	SP1583372	Kit, seat tilt cpl. (France)		1
5	SP1583210	Kit, backrest tilt cpl.		1
6	SP1582785	Battery cpl, Azalea		1
7	SP1582818	Control box CB06, cpl.		1
8	SP1582717	Handcontrol HB30, electric tilt		1
9	SP1582977	Plug cpl		1
10	SP1583029	Fixation, cable, Actuator LA23		1
11	SP1578477	Power cable, Actuator LA23		1
12	SP1578459	Power Cable EU		1
13	SP1578460	Power Cable UK		1
14	SP1578461	Power Cable AUSTRALIA		1

Backrest + seat tilt, electric, Azalea Max

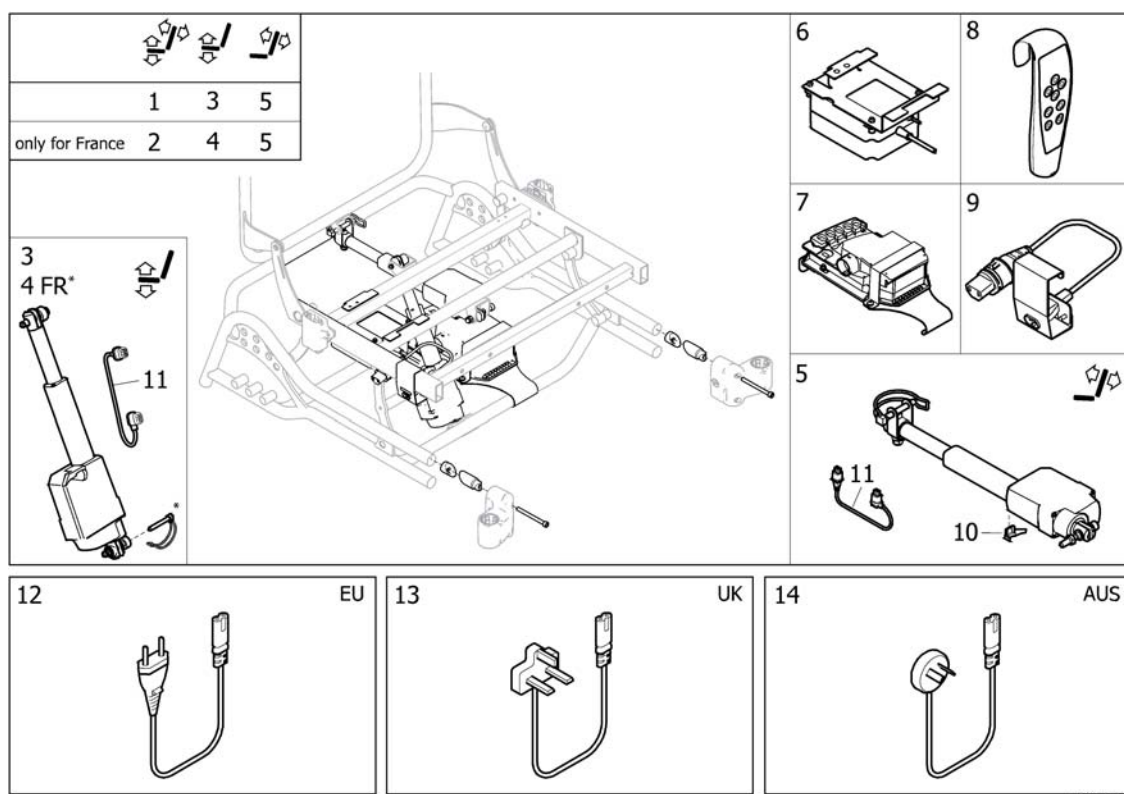

SPS1583209


Pos.	Part number	Description	Valid from → until	Qty
1	SP1583209	Kit, power tilt, Azalea Max		1
2	SP1583401	Kit, power tilt (France), Azalea Max		1
3	SP1583177	Kit, seat tilt cpl.		1
4	SP1583372	Kit, seat tilt cpl. (France)		1
5	SP1583211	Kit, backrest tilt cpl, Azalea Max		1
6	SP1582756	Battery cpl, Azalea Max		1
7	SP1582818	Control box CB06, cpl.		1
8	SP1582717	Handcontrol HB30, electric tilt		1
9	SP1582977	Plug cpl		1
10	SP1583029	Fixation, cable, Actuator LA23		1
11	SP1578477	Power cable, Actuator LA23		1
12	SP1578459	Power Cable EU		1
13	SP1578460	Power Cable UK		1
14	SP1578461	Power Cable AUSTRALIA		1

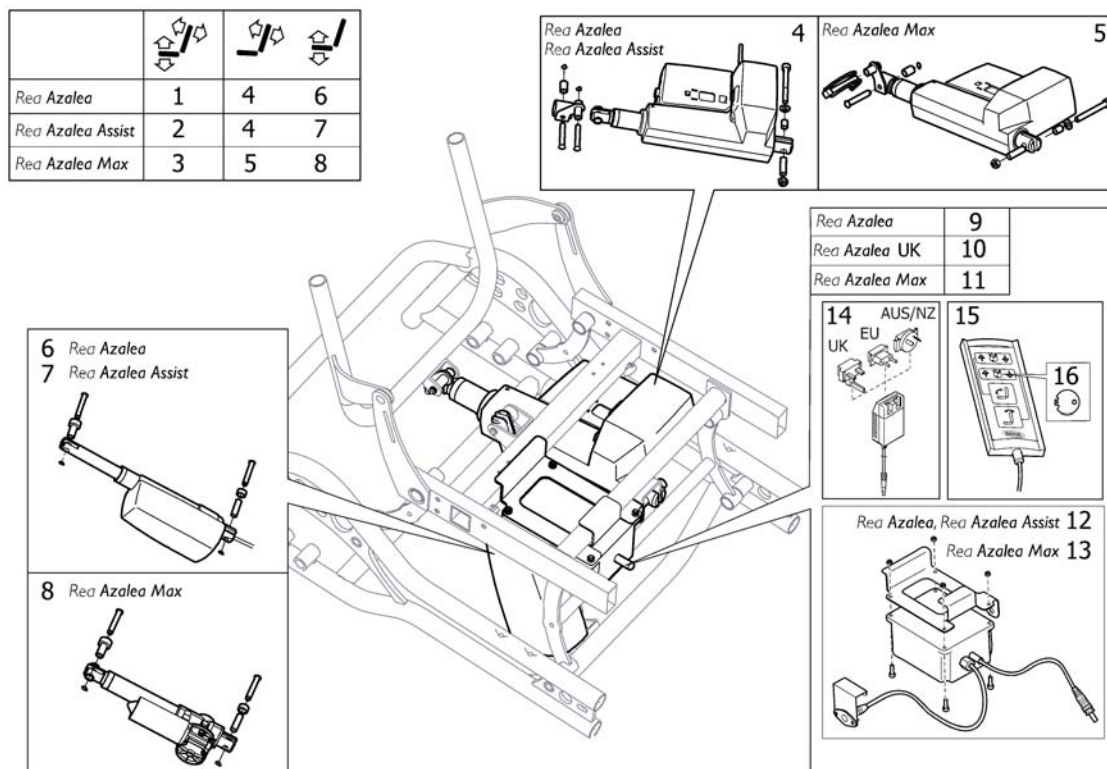Backrest + seat tilt, electric (=>2015.08)

Pos.	Part number	Description	Valid from → until	Qty
1	1484820	KIT EL-TILT, AZALEA		1
2	1483996	KIT EL-TILT, AZALEA ASSIST		1
3	1524790	KIT EL-TILT, AZALEA MAX,		1
4	1492904	PISTON LA31 CPL, BACKREST, AZALEA		1
5	1526825	ACTUATOR LA31, CPL, BACKREST		1
6	1493020	KIT EL-TILT UNIT, AZALEA		1
7	1492938	PISTON LA28 CPL, SEAT, AZALEA		1
8	1535278	ACTUATOR LINK LA28 CPL SEAT, ELTILT		1
9	1492939	KIT, STEERING UNIT		1
10	1508985	KIT, STEERING UNIT UK		1
11	1526826	KIT, STEERING UNIT, AZALEA MAX		1
12	1493097	BATTERY CPL., AZALEA		1
13	1526827	BATTERY CPL, AZALEA MAX		1
14	SP1571940	CHARGER CH01 INCL. CONNECTORS		1
15	1542453	HAND CONTROL HL70 <i>Replaces 1492696</i>		1
16	1542455	KEY, HAND CONTROL HL70		1

Castor attachment, fixed


	
1	2

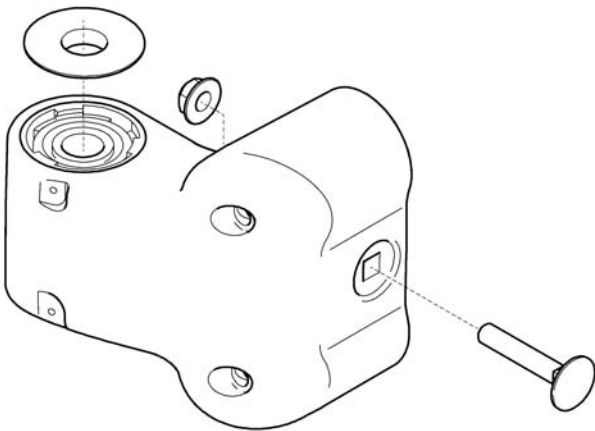

SP25352000

Pos.	Part number	Description	Valid from → until	Qty
1	5254309	CASTOR HOUSING, PAIR		1
2	5254308	CASTOR HOUSING FIXED		2


Kit, Azalea Tall

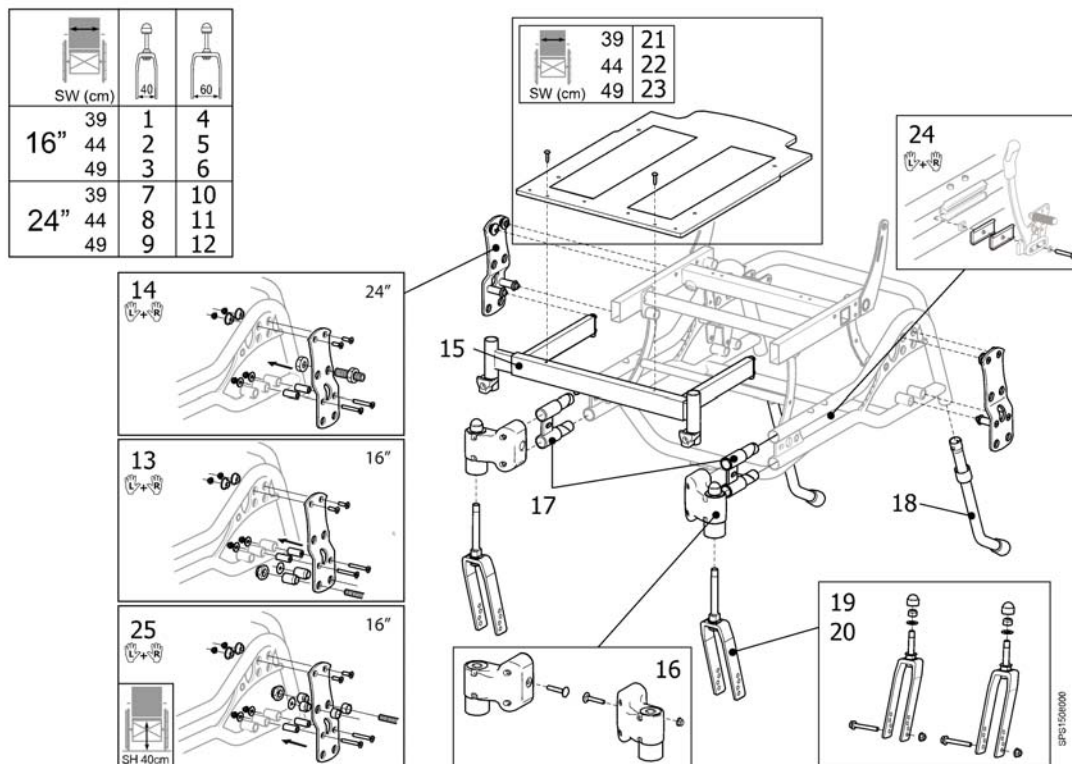

Pos.	Part number	Description	Valid from → until	Qty
1	1508000	AZALEA TALL 16", 39 CPL KIT, NARROW FORK		1
2	1508001	AZALEA TALL 16", 44 CPL KIT, NARROW FORK		1
3	1508002	AZALEA TALL 16", 49 CPL KIT, NARROW FORK		1
4	1508020	AZALEA TALL 16", 39 CPL KIT, WIDE FORK		1
5	1508021	AZALEA TALL 16", 44 CPL KIT, WIDE FORK		1
6	1508022	AZALEA TALL 16", 49 CPL KIT, WIDE FORK		1
7	1507958	AZALEA TALL 39 CPL KIT, NARROW FORK		1
8	1507998	AZALEA TALL 44 CPL KIT, NARROW FORK		1
9	1507999	AZALEA TALL 49 CPL KIT, NARROW FORK		1
10	1508017	AZALEA TALL 39 CPL KIT, WIDE FORK		1
11	1508018	AZALEA TALL 44 CPL KIT, WIDE FORK		1
12	1508019	AZALEA TALL 49 CPL KIT, WIDE FORK		1
13	1508006	DRIVE WHEEL ATTACH. , 16" PAIR		1
14	1508007	DRIVE WHEEL ATTACH. , 24" PAIR		1
15	1507966-0509	LEGREST ATTACH. 39 CPL, ADJUSTABLE		1
15	1507967-0509	LEGREST ATTACH. 44 CPL, ADJUSTABLE		1
15	1507968-0509	LEGREST ATTACH. 49 CPL, ADJUSTABLE		1
17	1484535	CHASSIS EXTENDER KPL, PAIR, AZALEA		1
18	1508008	ANTI-TIP DEVICE PAIR, TALL		1
16	1508003	CASTOR HOUSING PAIR,		1
19	1508011	CASTOR FORK CPL PAIR, NARROW		1
20	1508010	CASTOR FORK CPL PAIR, WIDE		1
21	1508138	SEATPLATE 350X470 CPL.		1
22	1508139	SEATPLATE 420X470 CPL.		1
23	1508140	SEATPLATE 450X470 CPL.		1
24	5207648	WIDTH EXT 10MM, BRAKE		1
25	1508785	ATTACHMENT PLATES 16" PAIR		1

Seat cushion, Flo-shape

SW (mm)	TR26	TR35	TR36
Azalea			
340	1	2	3
390	4	5	6
390 +50	4	5	6
390 +100	7	8	9
440	7	8	9
440 +50	7	8	9
440 +100	10	11	12
490	10	11	12
490 +50	10	11	12
490 +100	10	11	12
Bellis			
Small 410	4	5	
Medium 460	7	8	
Large 510	10	11	
Clematis			
390	1	2	
440	4	5	
490	7	8	
Dahlia			
390	4	5	6
390 +50	4	5	6
390 +100	7	8	9
440	7	8	9
440 +50	7	8	9
440 +100	10	11	12
490	10	11	12
490 +50	10	11	12
490 +100	10	11	12

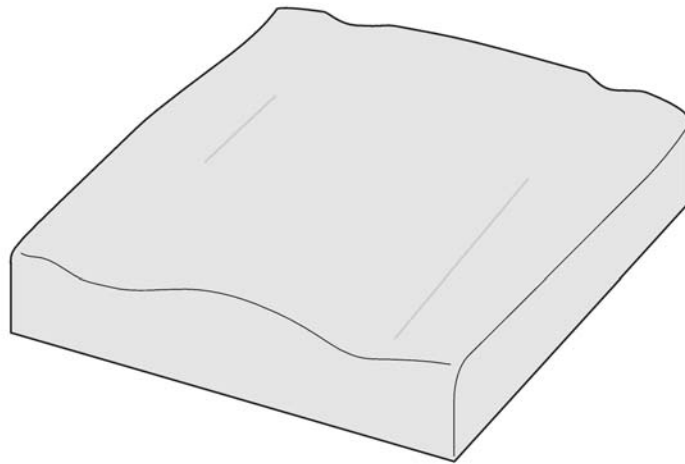


	Färg	Colour	Farbe	Couleur
TR26	dartex svart	dartex black	dartex schwarz	dartex noir
TR35	plysch svart	plysch black	plüsch schwarz	velours noir
TR36	spacer svart	spacer black	spacer schwarz	spacer noir

000005-15-01

Pos.	Part number	Description	Valid from → until	Qty
1	1509920-TR26	SEATCUSHION, FLOSHAPE, 355X450, DARTEX BLACK		1
2	1509920-TR35	SEATCUSHION, FLOSHAPE, 355X450, PLUSH BLACK		1
3	SP1509920-TR36	SEAT CUSHION, FLOSHAPE, 355X450, SPACER BLACK		1
4	1508055-TR26	Seat cushion Flo-shape, 405x535 mm, Dartex black (TR26)		1
5	1508055-TR35	SEAT CUSHION 405X535, FLOSHAPE, PLUSH BLACK		1
6	SP1508055-TR36	SEAT CUSHION, FLOSHAPE, 405X535, SPACER BLACK		1
7	1508057-TR26	Seat cushion Flo-shape, 455x535 mm, Dartex black (TR26)		1
8	1508057-TR35	SEAT CUSHION 455X535, FLOSHAPE, PLUSH BLACK		1
9	SP1508057-TR36	SEAT CUSHION, FLOSHAPE, 455X535, SPACER BLACK		1
10	1508059-TR26			1
11	1508059-TR35	SEAT CUSHION 505X535, FLOSHAPE, PLUSH BLACK		1
12	SP1508059-TR36	SEAT CUSHION, FLOSHAPE, 505X535, SPACER BLACK		1

Seat cushion, Flo-shape =>2012-04-01

				TR23	TR32	TR23	TR32
	SW (cm)						
Azalea							
34	34-44	1	2	11	12		
39	39	3	4	13	14		
39	44	5	6	15	16		
39	49	5	6	15	16		
44	44	5	6	15	16		
44	49	7	8	17	18		
44	54	7	8	17	18		
49	49	7	8	17	18		
49	54	9	10	19	20		
49	59	9	10	19	20		
Bellis							
Small	41	5	6	15	16		
Medium	46	7	8	17	18		
Large	51	9	10	19	20		
Clematis							
-	39	3	4	13	14		
-	44	5	6	15	16		
-	49	7	8	17	18		

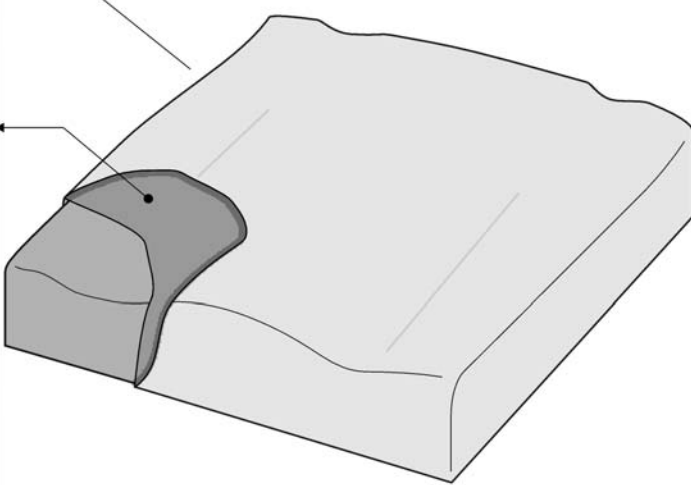


	Färg	Colour	Farbe	Couleur
TR23	dartex grå	dartex grey	dartex grau	dartex gris
TR32	plysch grå PRIMA	plush grey PRIMA	Rusch grau PRIMA	velours gris PRIMA

SP200003

Pos.	Part number	Description	Valid from → until	Qty
1	1509920	CUSHION FLO-SHAPE 330X450, SEAT TR23		1
2	1508976	CUSHION FLO-SHAPE 330X450, SEAT TR32		1
3	1508053	Flo-Shape cushion SW 355 x SD 535 Dartex		1
4	1508054	Flo-Shape cushion SW 355 x SD 535 Grey plush		1
5	1508055	Flo-Shape cushion SW 405 x SD 535 Dartex		1
6	1508056	Flo-Shape cushion SW 405 x SD 535 Grey plush		1
7	1508057	Flo-Shape cushion SW 455 x SD 535 Dartex		1
8	1508058	Flo-Shape cushion SW 455 x SD 535 Grey plush		1
9	1508059	CUSHION FLOSHAPE 505X535,SEAT,TR23		1
10	1508060	CUSHION FLOSHAPE 505X535,SEAT,TR32		1
11	1508977	UPHOLSTERY 330X450 SEAT, FLO-SHAPE TR23		1
12	1510626	UPHOLSTERY 330X450 SEAT, FLO-SHAPE TR32		1
13	1508081	UPHOLSTERY, SEAT, FLO-SHAPE 355X535, DARTEX GREY		1
14	1508082	UPHOLSTERY, SEAT, FLO-SHAPE 355X535, PLUSH GREY PRIMA		1
15	1508083	UPHOLSTERY, SEAT, FLO-SHAPE 405X535, DARTEX GREY		1
16	1508084	UPHOLSTERY, SEAT, FLO-SHAPE 405X535, PLUSH GREY PRIMA		1
17	1508085	UPHOLSTERY, SEAT, FLO-SHAPE 455X535, DARTEX GREY		1
18	1508086	UPHOLSTERY, SEAT, FLO-SHAPE 455X535, PLUSH GREY PRIMA		1
19	1508087	UPHOLSTERY, SEAT, FLO-SHAPE 505X535, DARTEX GREY		1
20	1508088	UPHOLSTERY, SEAT, FLO-SHAPE 505X535, PLUSH GREY PRIMA		1

Seat cushion, Vicair multifunctional

 SW (mm)		
Azalea		
390		1
390 +50		2
390 +100		3
440		2
440 +50		3
440 +100		4
490		3
490 +50		4
490 +100		4
Bellis		
Small 410		2
Medium 460		3
Large 510		4
Dahlia		
390		1
390 +50		2
390 +100		3
440		2
440 +50		3
440 +100		4
490		3
490 +50		4
490 +100		4

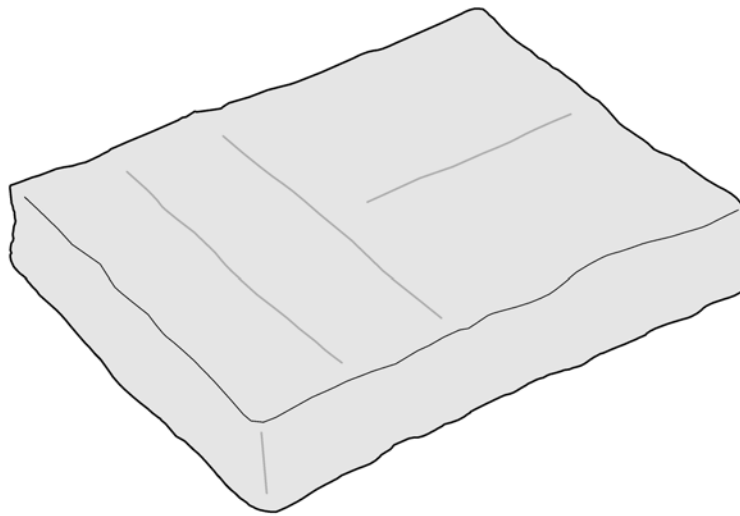


5951497347

Pos.	Part number	Description	Valid from → until	Qty
1	1497347	SEAT CUSHION 370X520X110, MULTI-FUNCTIONAL		1
2	1497348	SEAT CUSHION 400X520X110, MULTI-FUNCTIONAL		1
3	1497349	SEAT CUSHION 450X520X110, MULTI-FUNCTIONAL		1
4	1497387	SEAT CUSHION 500X520X110, MULTI-FUNCTIONAL		1

Seat cushion, Flo-tech contour Visco (only for France)


	SW (cm)	39	+5	+10	44	+5	+10	49	+5	+10
Azalea		2	2	3	3	3	4	4	4	4
Cleamtis		1			2			3		

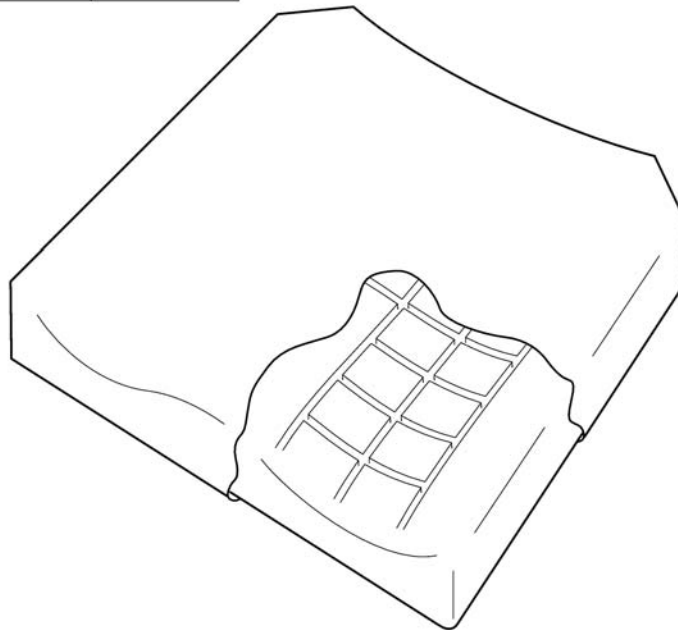

SPS 1501122

Pos.	Part number	Description	Valid from → until	Qty
1	1501120	CUSHION FLO-TECH 355X535 CONTOUR MEDIUM VISCO W.C		1
2	1501121	CUSHION FLO-TECH 405X535 CONTOUR FIRM VISCO W.COV		1
3	1501122	CUSHION FLO-TECH 455X535 CONTOUR FIRM VISCO W.COV		1
4	1501123	CUSHION FLO-TECH 505X535 CONTOUR V.FIRM VISCO W.C		1

	
SW (mm)	
Azalea	TR26 TR23
390	1
440	2
490	3
Dahlia	TR26 TR23
390	1
390 +50	2
390 +100	3
440	2
440 +50	3
440 +100	4
490	3
490 +50	4
490 +100	4
Clematis	TR26 TR23
390	1 5
440	2 6
490	3 7

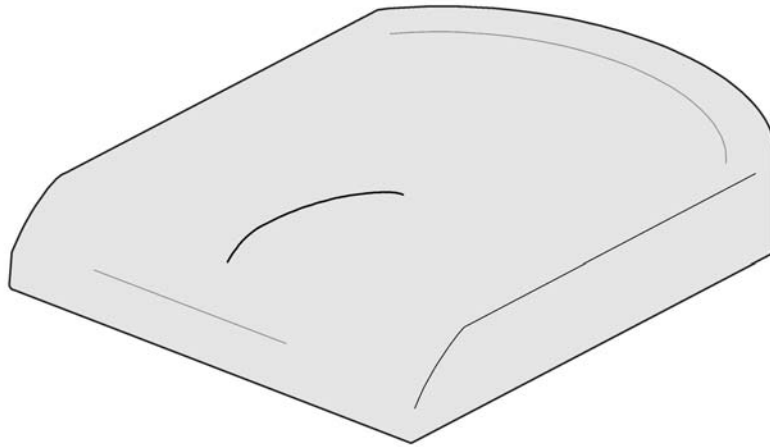


	Färg	Colour	Farbe	Couleur
TR26	dartex svart	dartex black	Dartex schwarz	dartex noir
→2012-04-01				
TR23	dartex grå	dartex grey	Dartex grau	dartex gris

6491131565

Pos.	Part number	Description	Valid from → until	Qty
1	SP1435856-TR26	SEAT CUSHION 365X490,MISTRAL 2-LAYER TR26		1
2	SP1435857-TR26	SEAT CUSHION 415X490,MISTRAL 2-LAYER TR26		1
3	SP1435858-TR26	SEAT CUSHION 465X490,MISTRAL 2-LAYER TR26		1
4	1435859-TR26	Seat cushion Mistral 515X490 mm, Dartex Black		1
5	1435856	Mistral shape cushion SW 365 x SD 490 Dartex		1
6	1435857	Mistral shape Cushion SW415 x SD490mm - Dartex		1
7	1435858	Mistral shape cushion SW 465 x SD 490 Dartex		1

Seat cushion, Azalea Max

	TR26 TR35	
	550	1 2
	610	3 4
→2012-04-01		
	TR23 TR32	
	550	5 6
	610	7 8

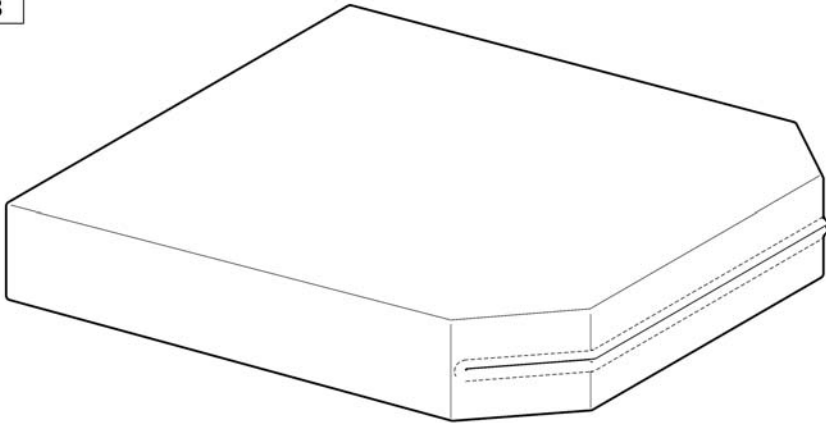


	Färg	Colour	Farbe	Couleur
TR26	dartex svart	dartex black	Dartex schwarz	dartex noir
TR35	plysch svart	plush black	Plüsch schwarz	velours noir
TR23	dartex grå	dartex grey	Dartex grau	dartex gris
TR32	plysch grå	plush grey	Plüsch grau	velours gris

SP53135096


Pos.	Part number	Description	Valid from → until	Qty
1	1536595-TR26	SEAT CUSHION 550X635, DARTEX BLACK		1
2	1536595-TR35	SEAT CUSHION 550X635, PLUSH BLACK		1
3	1536596-TR26	SEAT CUSHION 610X635, DARTEX BLACK		1
4	1536596-TR35	SEAT CUSHION 610X635, PLUSH BLACK		1
5	1536595	SEAT CUSHION 550X635, DARTEX GREY		1
6	1536597	SEAT CUSHION 550X635, PLUSH GREY		1
7	1536596	SEAT CUSHION 610X635, DARTEX GREY		1
8	1536598	SEAT CUSHION 610X635, PLUSH GREY		1


 SW (mm)	
Azalea	
340	1
390	2
390 +50	3
390 +100	4
440	3
440 +50	4
440 +100	4
490	4
490 +50	4
490 +100	4
Bellis	
Small 410	2
Medium 460	3
Large 510	4
Dahlia	
390	2
390 +50	3
390 +100	4
440	3
440 +50	4
440 +100	4
490	4
490 +50	4
490 +100	4

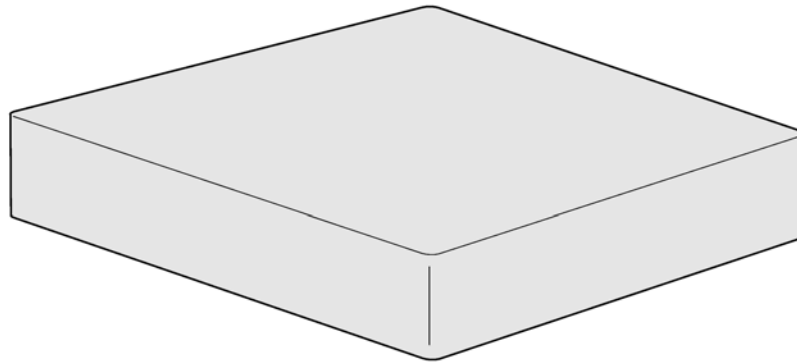

1510636


Pos.	Part number	Description	Valid from → until	Qty
1	1510636	INCONTINENCE COVER 330X450, BLACK		1
2	5253078-TR26	INCONTINENCE COVER 380X500, BLACK		1
3	5253079-TR26	Seat Cover, Comfort		1
4	5253080-TR26	INCONTINENCE COVER 480X500, BLACK		1
5	5253617-TR26	INCONTINENCE COVER 530X500, BLACK		1
6	1526617	INCONTINENCE COVER 550X635, BLACK		1
7	1526616	INCONTINENCE COVER 610X635, BLACK		1

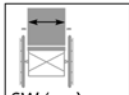	
SW (cm)	
39	1
44	2
49	3

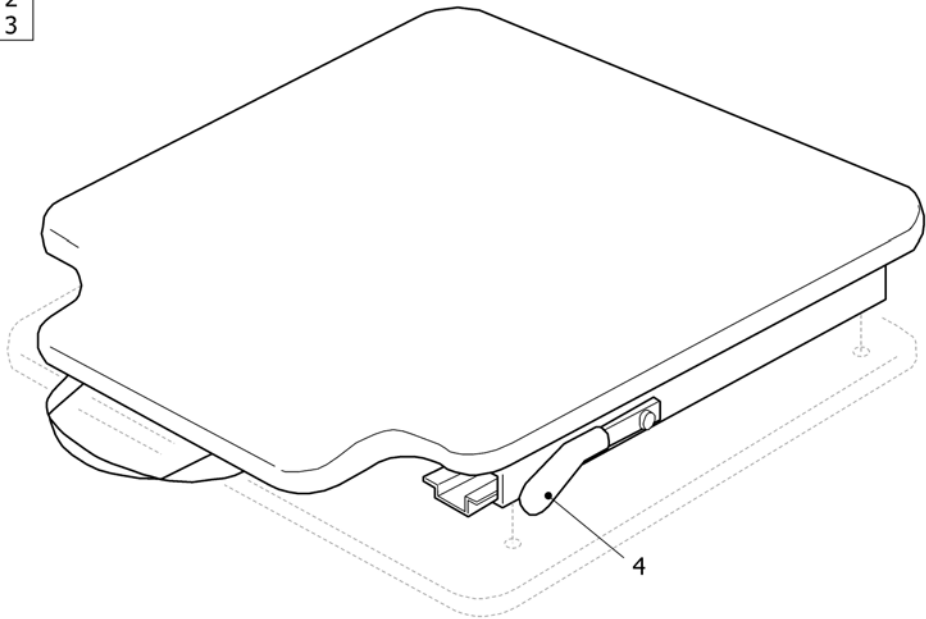


SP1484576

Pos.	Part number	Description	Valid from → until	Qty
1	SP1484576	Sliding seat 390		1
2	SP1484577	Sliding seat 440		1
3	SP1484578	Sliding seat 490		1
4	SP1492331	HANDLE, SLIDING SEAT, GREEN		1

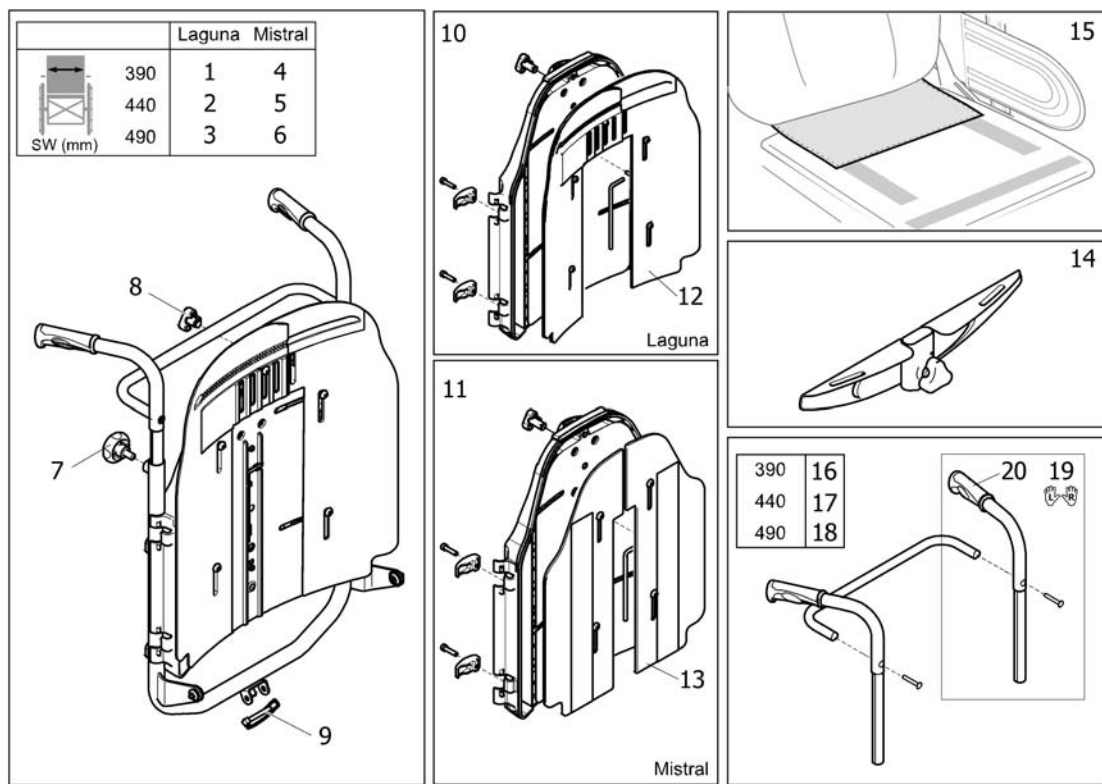


Pos.	Part number	Description	Valid from → until	Qty
1	SP1554115	BACKREST 390 CPL, FLEX3, LAGUNA		1
2	SP1554116	BACKREST 440 CPL, FLEX3, LAGUNA		1
3	SP1554117	BACKREST 490 CPL, FLEX3, LAGUNA		1
4	SP1562152	BACKREST 390 CPL, FLEX3, MISTRAL		1
5	SP1562153	BACKREST 440 CPL, FLEX3, MISTRAL		1
6	SP1562154	BACKREST 490 CPL, FLEX3, MISTRAL		1
7	SP1518478	Handwheel M10, pair		1
8	1497411	HANDWHEEL M12 LONG NECK/HEADREST		1
9	1439936	Pin with Loop 8X60		1
10	SP1562142	FLEX 3 HARD BACKREST, KIT LAGUNA		1
11	SP1562143	FLEX 3 HARD BACKREST, KIT MISTRAL 2		1
12	1543737	ADJUSTMENT PLATE CPL, FLEX3		1
13	1543742	ADJUSTMENT PLATE CPL, MISTRAL, FLEX3		1
14	SP1559963	Attachment, X-harness, Flex3		1
15	SP1543730	Cover, backrest/seat		1
16	SP1571966	PUSH HANDLES 390, FIXED CPL.		1
17	SP1571967	PUSH HANDLES 440, FIXED CPL.		1
18	SP1571968	PUSH HANDLES 490, FIXED CPL.		1
19	SP1572734	PUSH HANDLE CPL., PAIR		1
20	1537634	Rubber Handle 22mm, Pair		1

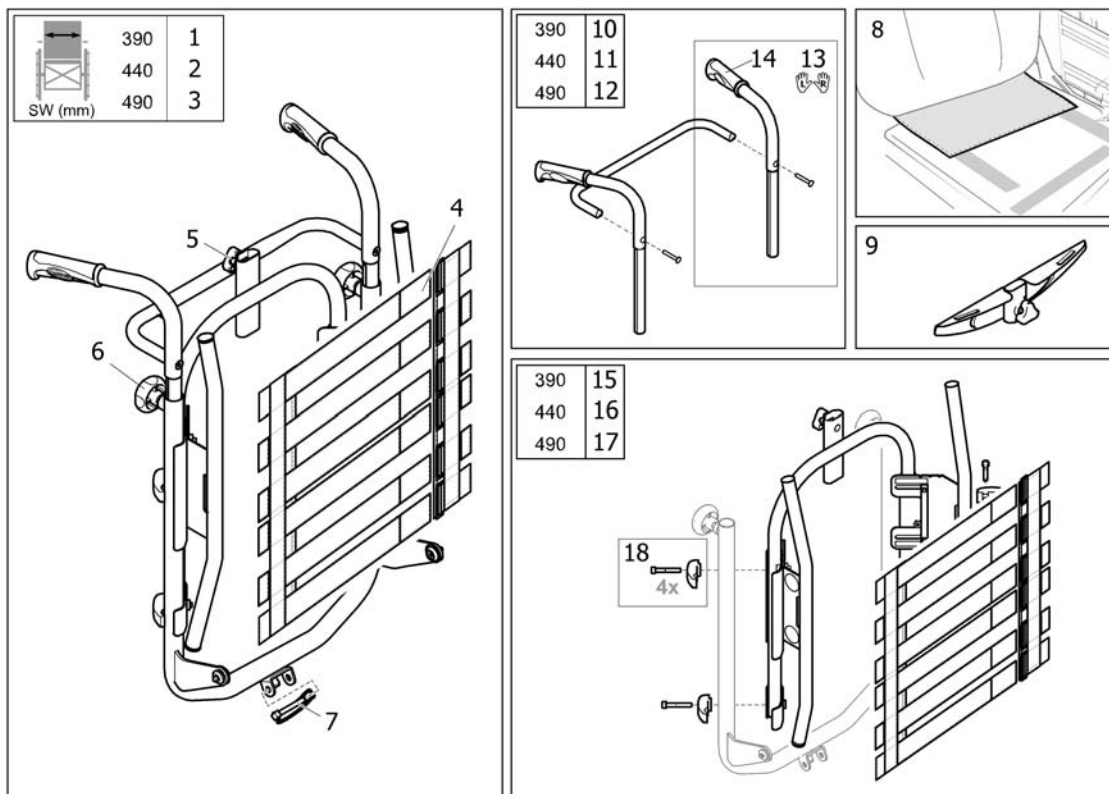Backrest "Flex3", tension adj.


Pos.	Part number	Description	Valid from → until	Qty
1	SP1554118	FLEX 3 TENSION BACKREST, KIT 39		1
2	SP1554119	FLEX 3 TENSION BACKREST, KIT 44		1
3	SP1554120	FLEX 3 TENSION BACKREST, KIT 49		1
4	1549281	BACKREST BODY, ADJ.		1
5	1444175	HANDWHEEL UNLOSEABLE CPL CPL		1
6	SP1518478	Handwheel M10, pair		1
7	1439936	Pin with Loop 8X60		1
8	SP1543730	Cover, backrest/seat		1
9	SP1559963	Attachment, X-harness, Flex3		1
10	SP1571966	PUSH HANDLES 390, FIXED CPL.		1
11	SP1571967	PUSH HANDLES 440, FIXED CPL.		1
12	SP1571968	PUSH HANDLES 490, FIXED CPL.		1
13	SP1572734	PUSH HANDLE CPL., PAIR		1
14	1537634	Rubber Handle 22mm, Pair		1
15	SP1579675	No description available.		1
16	SP1579692	No description available.		1
17	SP1579693	No description available.		1
18	SP1580557	Mounting Hardware (Kit of 4)		1

Backrest, Azalea minor

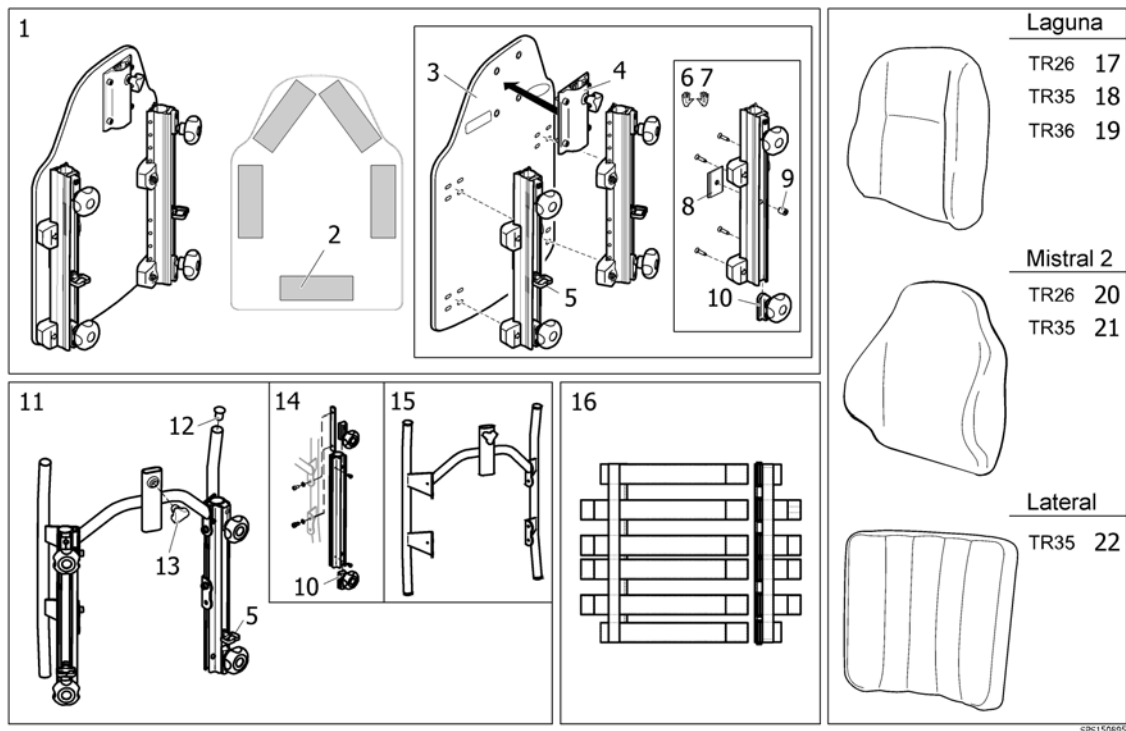


SPS1508958


Pos.	Part number	Description	Valid from → until	Qty
1	1510891	Backrest plate "Flex2" fixed, cpl.		1
2	5254139	Velcro hook 150mm		1
3	1508958	Backrest plate 34 cpl.		1
4	1446010	Bracket for Neck-/Headrest Flex2		1
5	1508131	Wire holder		1
6	1446009	Rail LH, backrest FLEX2, fixed		1
7	1446008	Rail RH, backrest FLEX2, fixed		1
8	1446015	Stop, backrest rail		1
9	5723904	Screw MF6S M6x8		1
10	5253088	Fixation block		1
11	1543577	Backrest adjustable 292 cpl.		1
12	5716478	Plastic Sleeve, black		1
13	1444175	HANDWHEEL UNLOSEABLE CPL CPL		1
14	1445848	Backrest Rail, cpl.		1
15	1444297-0509	Backrest frame 292		1
16	1445694	Backrest, tension adjustable, SW up to 49cm		1
17	SP1508972-TR26	CUSHION, BACKREST LAGUNA 322X450, DARTEX BLACK		1
18	SP1508972-TR35	CUSHION, BACKREST LAGUNA 322X450, PLUSH BLACK		1
19	SP1508972-TR36	CUSHION, BACKREST LAGUNA 322X450, SPACER BLACK		1
20	SP1431668-TR26	CUSHION, BACKREST MISTRAL2 355X510, DARTEX BLACK		1
21	SP1431668-TR35	CUSHION, BACKREST MISTRAL2 355X510, PLUSH BLACK		1
22	SP1564092	COVER BACKREST 380 CPL.		1

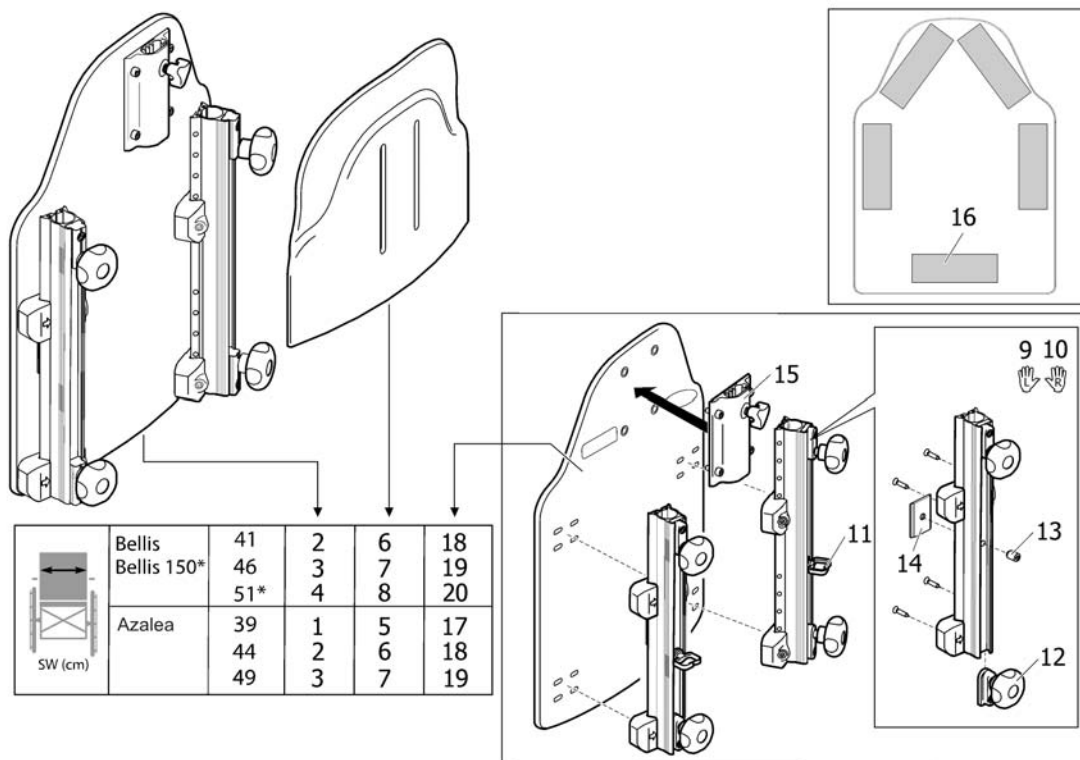Backrest "Flex2", fixed (->2013)

Pos.	Part number	Description	Valid from → until	Qty
1	1440151	BACKREST PLATE Flex2 405X485 CPL		1
2	1440152	BACKREST PLATE Flex2 455X485 CPL		1
3	1440153	BACKREST PLATE Flex2 505X485 CPL		1
4	1440154	BACKREST PLATE "FLEX2" 505x485 CPL		1
5	1440147	EXTENDER 405X290 CPL, BACKREST FLEX2		1
6	1440148	EXTENDER 455X290 CPL, BACKREST FLEX2		1
7	1440149	EXTENDER 505X290 CPL, BACKREST FLEX2		1
8	1440150	EXTENDER 505X290 CPL, BACKREST FLEX 2		1
9	1446009	Rail LH, backrest FLEX2, fixed		1
10	1446008	Rail RH, backrest FLEX2, fixed		1
11	1508131	Wire holder		1
12	5253088	Fixation block		1
13	5723904	Screw MF6S M6x8		1
14	1446015	Stop, backrest rail		1
15	1446010	Bracket for Neck-/Headrest Flex2		1
16	5254139	Velcro hook 150mm		1
17	1507942	BACKREST PLATE 355X485, WITH VELCRO		1
18	1507943	BACKREST PLATE 405X485, WITH VELCRO		1
19	1507944	BACKREST PLATE 455X485, WITH VELCRO		1
20	1507945	BACKREST PLATE 505X485, WITH VELCRO		1

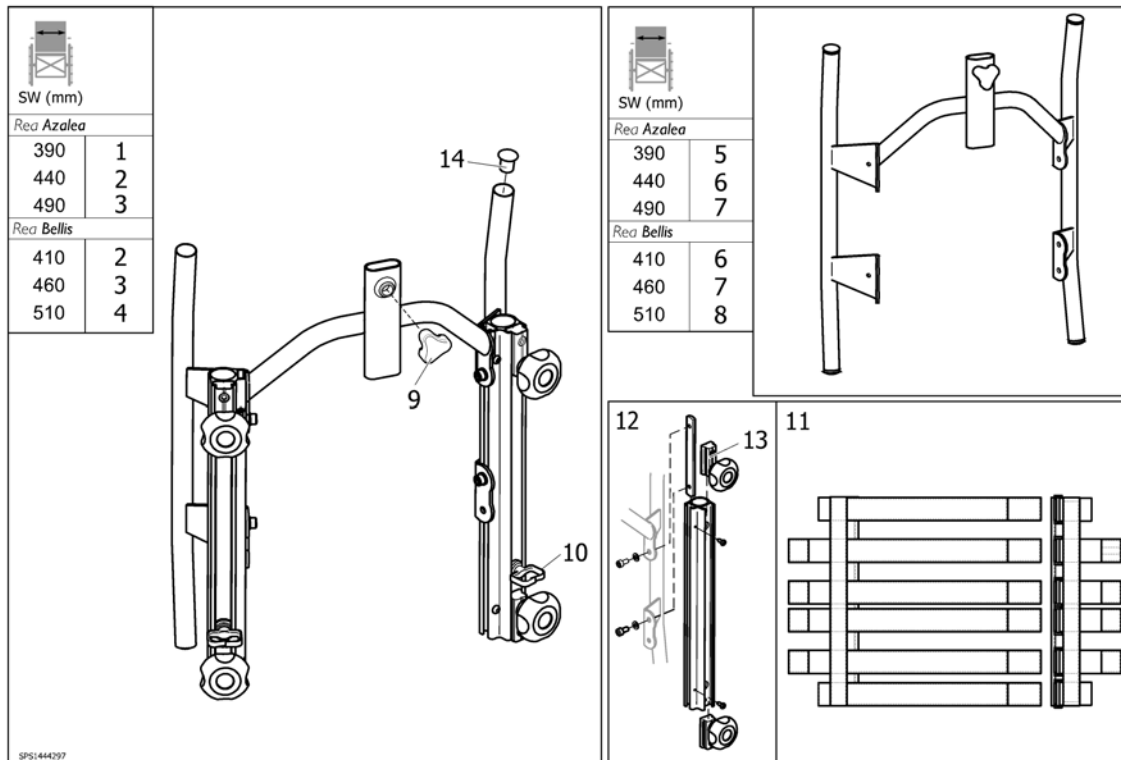
Backrest "Flex2", adj. (->2013)

Pos.	Part number	Description	Valid from → until	Qty
1	1445697	Backrest, adjustable 292 (SW 390mm), cpl.		1
2	1445698	Backrest, adjustable 342 (SW 440mm), cpl.		1
3	1445699	Backrest, adjustable 392 (SW 490mm), cpl.		1
4	1445700	BACKREST 442, ADJUSTABLE, CPL.		1
5	1445844-0509	Backrest Frame SW 390mm		1
6	1445845-0509	Backrest Frame SW 440mm		1
7	1445846-0509	Backrest Frame SW 490mm		1
8	1445847-0509	BACKREST FRAME 442 CPL.		1
9	1444175	HANDWHEEL UNLOSEABLE CPL CPL		1
10	1508131	Wire holder		1
11	1445694	Backrest, tension adjustable, SW up to 49cm		1
12	1445848	Backrest Rail, cpl.		1
13	5253088	Fixation block		1
14	5716478	Plastic Sleeve, black		1

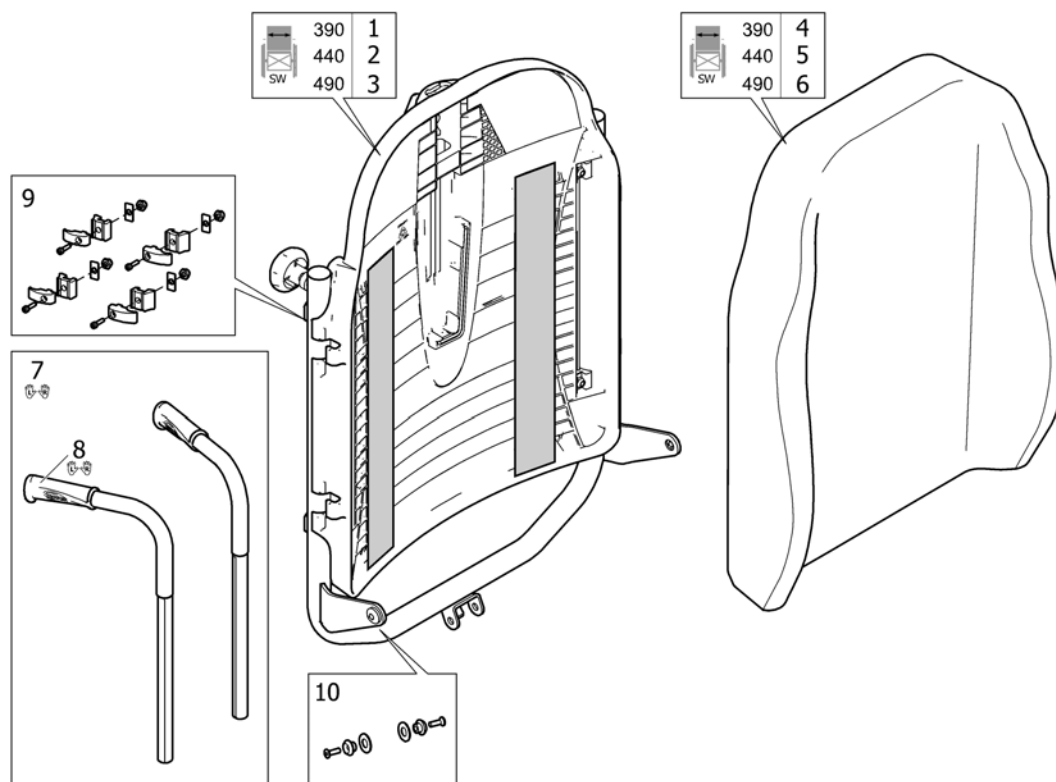Backrest, Laguna (France only)

54163556

Pos.	Part number	Description	Valid from → until	Qty
1	SP1571973	BACKREST 390, LAGUNA, AZALEA		1
2	SP1571974	BACKREST 440, LAGUNA, AZALEA		1
3	SP1571975	BACKREST 490, LAGUNA, AZALEA		1
4	SP1439937-TR26	BACKREST CUSHION, 342X550, LAGUNA, DARTEX BLACK		1
5	SP1427119-TR26	Backrest Cushion Laguna, 402x550mm, Dartex black		1
6	SP1427120-TR26	Backrest Cushion Laguna, 452x550mm, Dartex black		1
7	SP1572734	PUSH HANDLE CPL., PAIR		1
8	SP1537634	HANDLE 22MM, PAIR		1
9	1518649	FITTING KIT, LAGUNA2		1
10	SP1579941	Screw kit, backrest stay		1

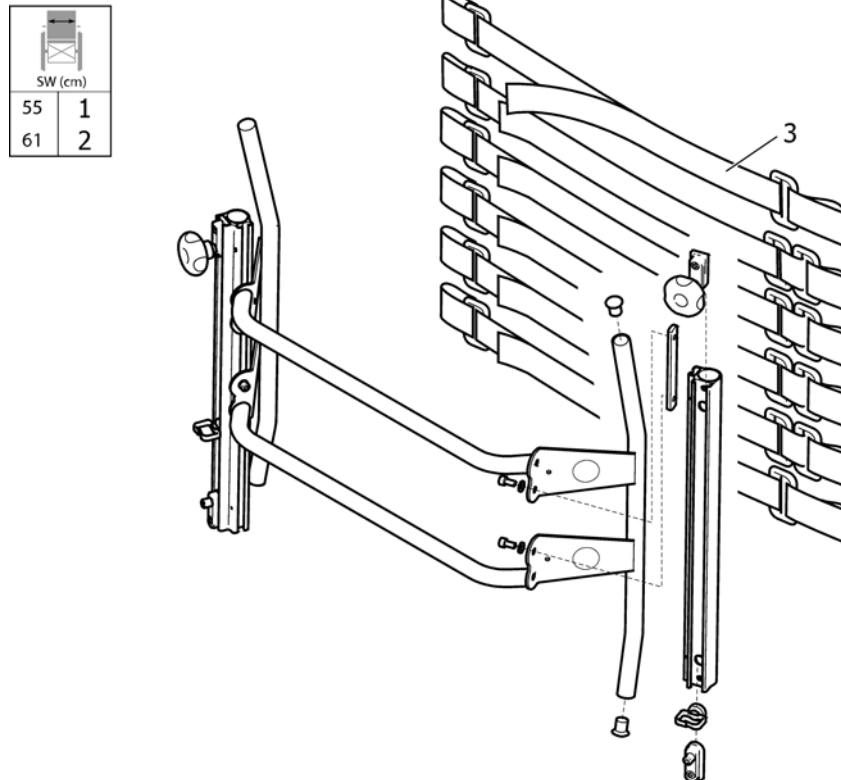Backrest, Azalea Max

Pos.	Part number	Description	Valid from → until	Qty
1	1526824	Backrest 55 Cpl, Azalea Max		1
2	1526823	Backrest 61 Cpl, Azalea Max		1
3	1526308	Backrest Body, Tension Adj, Azalea Max		1

Cushions, Flex3 backrest

												
BW + ext.	Mistral 2		Laguna			Passad 2			Shoulder high	Multi Functional	Lateral	
	TR26	TR35	TR26	TR35	TR36	TR26	TR35	TR36	TR26		TR26	TR35
390	1	2	9	10	11	21	22	23	33	36	39	40
390 +50	3	4	12	13	14	24	25	26	33	36	39	40
390 +100	5	6	15	16	17	24	25	26	33	36	39	40
440	3	4	12	13	14	24	25	26	33	37	41	42
440 +50	5	6	15	16	17	27	28	29	34	37	41	42
440 +100	7	8	18	19	20	27	28	29	34	37	41	42
490	5	6	15	16	17	27	28	29	34	38	43	44
490 +50	7	8	18	19	20	30	31	32	35	38	43	44
490 +100	-	-	-	-	-	-	-	-	-	38	43	44

45



	Färg	Colour	Farbe	Couleur
TR26	dartex svart	dartex black	dartex schwarz	dartex noir
TR35	plysch svart	plysch black	plüsch schwarz	velours noir
TR36	spacer svart	spacer black	spacer schwarz	spacer noir


860961565

Pos.	Part number	Description	Valid from → until	Qty
1	SP1431668-TR26	CUSHION, BACKREST MISTRAL2 355X510, DARTEX BLACK		1
2	SP1431668-TR35	CUSHION, BACKREST MISTRAL2 355X510, PLUSH BLACK		1
3	SP1424651-TR26	CUSHION, BACKREST MISTRAL2 405X510, DARTEX BLACK		1
4	SP1424651-TR35	CUSHION, BACKREST MISTRAL2 405X510, PLUSH BLACK		1
5	SP1424652-TR26	CUSHION, BACKREST MISTRAL2 455X510, DARTEX BLACK		1
6	SP1424652-TR35	CUSHION, BACKREST MISTRAL2 455X510, PLUSH BLACK		1
7	SP1424653-TR26	CUSHION, BACKREST MISTRAL2 505X510, DARTEX BLACK		1
8	SP1424653-TR35	CUSHION, BACKREST MISTRAL2 505X510, PLUSH BLACK		1
9	SP1427118-TR26	Backrest Cushion Laguna, 372x550mm, Dartex black		1
10	SP1427118-TR35	CUSHION, BACKREST LAGUNA 372X550, PLUSH BLACK		1
11	SP1427118-TR36	CUSHION, BACKREST LAGUNA 372X550, SPACER BLACK		1
12	SP1427119-TR26	Backrest Cushion Laguna, 402x550mm, Dartex black		1
13	SP1427119-TR35	BACKREST CUSHION 402X550, LAGUNA, PLUSH BLACK		1
14	SP1427119-TR36	BACKREST CUSHION 402X550, LAGUNA, SPACER BLACK		1
15	SP1427120-TR26	Backrest Cushion Laguna, 452x550mm, Dartex black		1
16	SP1427120-TR35	BACKREST CUSHION 452X550, LAGUNA, PLUSH BLACK		1
17	SP1427120-TR36	BACKREST CUSHION 452X550, LAGUNA, SPACER BLACK		1
18	SP1427121-TR26	BACKREST CUSHION 492X550, LAGUNA, DARTEX BLACK		1
19	SP1427121-TR35	BACKREST CUSHION 492X550, LAGUNA, PLUSH BLACK		1
20	SP1427121-TR36	BACKREST CUSHION 492X550, LAGUNA, SPACER BLACK		1
21	SP1429271-TR26	BACKREST CUSHION 440X550, PASSAD2, DARTEX BLACK		1
22	SP1429271-TR35	BACKREST CUSHION 440X550, PASSAD2, PLUSH BLACK		1
23	SP1429271-TR36	BACKREST CUSHION 440X550, PASSAD2, SPACER BLACK		1
24	SP1424647-TR26	BACKREST CUSHION 470X550, PASSAD2, DARTEX BLACK		1
25	SP1424647-TR35	BACKREST CUSHION 470X550, PASSAD2, PLUSH BLACK		1
26	SP1424647-TR36	BACKREST CUSHION 470X550, PASSAD2, SPACER BLACK		1
27	SP1424649-TR26	BACKREST CUSHION 520X550, PASSAD2, DARTEX BLACK		1
28	SP1424649-TR35	BACKREST CUSHION 520X550, PASSAD2, PLUSH BLACK		1
29	SP1424649-TR36	BACKREST CUSHION 520X550, PASSAD2, SPACER BLACK		1
30	SP1429278-TR26	BACKREST CUSHION 580X550, PASSAD2, DARTEX BLACK		1
31	SP1429278-TR35	BACKREST CUSHION 580X550, PASSAD2, PLUSH BLACK		1

Cushions, Flex3 backrest

Pos.	Part number	Description	Valid from → until	Qty
32	SP1429278-TR36	BACKREST CUSHION 580X550, PASSAD2, SPACER BLACK		1
33	SP1444275-TR26	BACKREST CUSHION 440X650, SHOULDER HIGH, DARTEX BLACK		1
34	SP1444277-TR26	BACKREST CUSHION 520X650, SHOULDER HIGH, DARTEX BLACK		1
35	SP1444278-TR26	BACKREST CUSHION 580X650, SHOULDER HIGH, DARTEX BLACK		1
36	1435592	BACKREST CUSHION 370X540, VICAIR MF, DARTEX GREY		1
37	1445741	BACKREST CUSHION 400X540, VICAIR MF, DARTEX GREY		1
38	1445742	BACKREST CUSHION 450X540, VICAIR MF, DARTEX GREY		1
39	SP1543667	Backrest cushion Lateral 475x540mm, Dartex black (TR26)		1
40	SP1543668	COVER LATERAL 475X540, PLUSH BLACK		1
41	SP1543669	Backrest cushion Lateral 525x540mm, Dartex black (TR26)		1
42	SP1543670	COVER LATERAL 525X540, PLUSH BLACK		1
43	SP1543671	Backrest cushion Lateral 575x540mm, Dartex black (TR26)		1
44	SP1543672	COVER LATERAL 575X540, PLUSH BLACK		1
45	SP1513037	WEDGES, PAIR		1

Cushions, Flex2 backrest

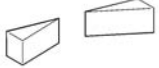 BW + ext.		 Mistral 2		 Laguna			 Passad 2			 Shoulder high		 Multi Functional		 Lateral	
		TR26	TR35	TR26	TR35	TR36	TR26	TR35	TR36	TR26					
340		1	2	9	10	11	-	-	-	-		-		-	58
390		3	4	12	13	14	25	26	27	37		41		44	45
390	+50	5	6	15	16	17	28	29	30	38		42		46	47
390	+100	7	8	18	19	20	31	32	33	39		43		48	49
440		5	6	15	16	17	28	29	30	38		42		46	47
440	+50	7	8	18	19	20	31	32	33	39		43		48	49
440	+100	-	-	21	22	-	34	35	36	40		-		50	51
490		7	8	18	19	20	31	32	33	39		43		48	49
490	+50	-	-	21	22	-	34	35	36	40		-		50	51
490	+100	-	-	23	24	-	-	-	-	-		-		52	53
550		-	-	21	22	-	-	-	-	-		-		50	51
550	+50	-	-	23	24	-	-	-	-	-		-		52	53
550	+100	-	-	-	-	-	-	-	-	-		-		54	55
610		-	-	-	-	-	-	-	-	-		-		52	53
610	+50	-	-	-	-	-	-	-	-	-		-		54	55
610	+100	-	-	-	-	-	-	-	-	-		-		56	57
59 							Färg		Colour		Farbe		Couleur		
							TR26	dartex svart	dartex black	dartex schwarz	dartex noir				
							TR35	plusch svart	plusch black	plüsch schwarz	velours noir				
							TR36	spacer svart	spacer black	spacer schwarz	spacer noir				


SP1506097


Pos.	Part number	Description	Valid from → until	Qty
1	SP1431668-TR26	CUSHION, BACKREST MISTRAL2 355X510, DARTEX BLACK		1
2	SP1431668-TR35	CUSHION, BACKREST MISTRAL2 355X510, PLUSH BLACK		1
3	SP1424651-TR26	CUSHION, BACKREST MISTRAL2 405X510, DARTEX BLACK		1
4	SP1424651-TR35	CUSHION, BACKREST MISTRAL2 405X510, PLUSH BLACK		1
5	SP1424652-TR26	CUSHION, BACKREST MISTRAL2 455X510, DARTEX BLACK		1
6	SP1424652-TR35	CUSHION, BACKREST MISTRAL2 455X510, PLUSH BLACK		1
7	SP1424653-TR26	CUSHION, BACKREST MISTRAL2 505X510, DARTEX BLACK		1
8	SP1424653-TR35	CUSHION, BACKREST MISTRAL2 505X510, PLUSH BLACK		1
9	SP1508972-TR26	CUSHION, BACKREST LAGUNA 322X450, DARTEX BLACK		1
10	SP1508972-TR35	CUSHION, BACKREST LAGUNA 322X450, PLUSH BLACK		1
11	SP1508972-TR36	CUSHION, BACKREST LAGUNA 322X450, SPACER BLACK		1
12	SP1427119-TR26	Backrest Cushion Laguna, 402x550mm, Dartex black		1
13	SP1427119-TR35	BACKREST CUSHION 402X550, LAGUNA, PLUSH BLACK		1
14	SP1427119-TR36	BACKREST CUSHION 402X550, LAGUNA, SPACER BLACK		1
15	SP1427120-TR26	Backrest Cushion Laguna, 452x550mm, Dartex black		1
16	SP1427120-TR35	BACKREST CUSHION 452X550, LAGUNA, PLUSH BLACK		1
17	SP1427120-TR36	BACKREST CUSHION 452X550, LAGUNA, SPACER BLACK		1
18	SP1427121-TR26	BACKREST CUSHION 492X550, LAGUNA, DARTEX BLACK		1
19	SP1427121-TR35	BACKREST CUSHION 492X550, LAGUNA, PLUSH BLACK		1
20	SP1427121-TR36	BACKREST CUSHION 492X550, LAGUNA, SPACER BLACK		1
21	SP1525073-TR26	BACKREST CUSHION 560X550, LAGUNA, DARTEX BLACK		1
22	SP1525073-TR35	BACKREST CUSHION 560X550, LAGUNA, PLUSH BLACK		1
23	SP1525075-TR26	CUSHION, BACKREST LAGUNA 620X550, DARTEX BLACK		1
24	SP1525075-TR35	CUSHION, BACKREST LAGUNA 620X550, PLUSH BLACK		1
25	SP1429271-TR26	BACKREST CUSHION 440X550, PASSAD2, DARTEX BLACK		1
26	SP1429271-TR35	BACKREST CUSHION 440X550, PASSAD2, PLUSH BLACK		1
27	SP1429271-TR36	BACKREST CUSHION 440X550, PASSAD2, SPACER BLACK		1
28	SP1424647-TR26	BACKREST CUSHION 470X550, PASSAD2, DARTEX BLACK		1
29	SP1424647-TR35	BACKREST CUSHION 470X550, PASSAD2, PLUSH BLACK		1
30	SP1424647-TR36	BACKREST CUSHION 470X550, PASSAD2, SPACER BLACK		1
31	SP1424649-TR26	BACKREST CUSHION 520X550, PASSAD2, DARTEX BLACK		1

Cushions, Flex2 backrest

Pos.	Part number	Description	Valid from → until	Qty
32	SP1424649-TR35	BACKREST CUSHION 520X550, PASSAD2, PLUSH BLACK		1
33	SP1424649-TR36	BACKREST CUSHION 520X550, PASSAD2, SPACER BLACK		1
34	SP1429278-TR26	BACKREST CUSHION 580X550, PASSAD2, DARTEX BLACK		1
35	SP1429278-TR35	BACKREST CUSHION 580X550, PASSAD2, PLUSH BLACK		1
36	SP1429278-TR36	BACKREST CUSHION 580X550, PASSAD2, SPACER BLACK		1
37	SP1444275-TR26	BACKREST CUSHION 440X650, SHOULDER HIGH, DARTEX BLACK		1
38	SP1444276-TR26	BACKREST CUSHION 470X650, SHOULDER HIGH, DARTEX BLACK		1
39	SP1444277-TR26	BACKREST CUSHION 520X650, SHOULDER HIGH, DARTEX BLACK		1
40	SP1444278-TR26	BACKREST CUSHION 580X650, SHOULDER HIGH, DARTEX BLACK		1
41	1445741	BACKREST CUSHION 400X540, VICAIR MF, DARTEX GREY		1
42	1445742	BACKREST CUSHION 450X540, VICAIR MF, DARTEX GREY		1
43	1445743	BACKREST CUSHION 500X540, VICAIR MF, DARTEX GREY		1
44	SP1574717	COVER LATERAL 425X540, DARTEX BLACK		1
45	SP1574718	COVER LATERAL 425X540, PLUSH BLACK		1
46	SP1543667	Backrest cushion Lateral 475x540mm, Dartex black (TR26)		1
47	SP1543668	COVER LATERAL 475X540, PLUSH BLACK		1
48	SP1543669	Backrest cushion Lateral 525x540mm, Dartex black (TR26)		1
49	SP1543670	COVER LATERAL 525X540, PLUSH BLACK		1
50	SP1543671	Backrest cushion Lateral 575x540mm, Dartex black (TR26)		1
51	SP1543672	COVER LATERAL 575X540, PLUSH BLACK		1
52	SP1543673	COVER, LATERAL 625X540, DARTEX BLACK		1
53	SP1543674	COVER, LATERAL 625X540, PLUSH BLACK		1
54	SP1543675	COVER, LATERAL 665X540, DARTEX BLACK		1
55	SP1543676	COVER, LATERAL 665X540, PLUSH BLACK		1
56	SP1543677	COVER, LATERAL 725X540, DARTEX BLACK		1
57	SP1543678	COVER, LATERAL 725X540, PLUSH BLACK		1
58	SP1564092	COVER BACKREST 380 CPL.		1
59	SP1513037	WEDGES, PAIR		1

Cushions, Flex2 backrest =>2012-04-01

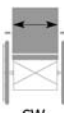								
		SW	Mistral 2	Laguna	Passad 2	Shoulder high	Multi Functional	Lateral
Assist 705 Silencio	400-500		2	7	13	17	21	25
	450-550		3	8	14	18	22	26
Bellis	410 (S)		2	7	13	17	21	25
	460 (M)		3	8	14	18	22	26
Bellis 150	510 (L)		4	9	15	19	23	27
Azalea	340		-	5	-	-	-	-
	390		1	6	12	16	20	24
	440		2	7	13	17	21	25
	490		3	8	14	18	22	26
	550		-	10	-	-	-	28
	610		-	11	-	-	-	29

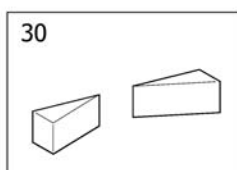

	Farg	Colour	Farbe	Couleur
TR23	dartex grå	dartex grey	Dartex grau	dartex gris
TR32	plysch grå PRIMA	plush grey PRIMA	Plüsch grau PRIMA	velours gris PRIMA


SPS 143 1668

Pos.	Part number	Description	Valid from → until	Qty
1	1431668	Cushion, Backrest Mistral2 355X510, Dartex Grey		1
1	1431669	Cushion, Backrest Mistral2 355X510, Plush Grey		1
2	1424651	Cushion, Backrest Mistral2 405X510, Dartex Grey		1
2	1435404	Cushion, Backrest Mistral2 405X510, Plush Grey		1
3	1424652	Cushion, Backrest Mistral2 455X510, Dartex Grey		1
3	1435405	Cushion, Backrest Mistral2 455X510, Plush Grey		1
4	1424653	Cushion, Backrest Mistral2 505X510, Dartex Grey		1
4	1435406	Cushion, Backrest Mistral2 505X510, Plush Grey		1
5	1508972	Cushion, Backrest Laguna 322X450, Dartex Grey		1
5	1510660	Cushion, Backrest Laguna 322X450, Plush Grey		1
6	1427118	Backrest cushion comfort Laguna (XS), 372x550 mm, Dartex grey (TR23)		1
6	1444339	Backrest Cushion Laguna, 372x550mm, Grey Plush		1
7	1427119	Backrest cushion comfort Laguna (S), 402x550 mm, Dartex grey (TR23)		1
7	1435798	Backrest Cushion Laguna, 402x550mm, Grey Plush		1
8	1427120	Backrest cushion comfort Laguna (M), 452x550 mm, Dartex grey (TR23)		1
8	1444340	Backrest Cushion Laguna, 452x550mm, Grey Plush		1
9	1427121	Backrest Cushion Laguna, 492x550mm, Dartex Grey		1
9	1444341	Backrest Cushion Laguna, 492x550mm, Grey Plush		1
10	1525073	Cushion, Backrest Laguna 560x550, Dartex Grey		1
10	1525074	Cushion, Backrest Laguna 560x550, Plush Grey Prima		1
11	1525075	Cushion, Backrest Laguna 620x550, Dartex Grey		1
11	1525076	Cushion, Backrest Laguna 620x550, Plush Grey		1
12	1429271	Backrest cushion comfort Passad2 (XS), 440x550 mm, Dartex gris (TR23)		1
12	1444328	Cushion, Backrest Passad2 440X550, Plush Grey		1
13	1424647	Backrest cushion comfort Passad2 (S), 470x550 mm, Dartex grey (TR23)		1
13	1444329	Cushion, Backrest Passad2 470X550, Plush Grey		1
14	1424649	Backrest cushion comfort Passad2 (M), 520x550 mm, Dartex grey (TR23)		1
14	1444330	Cushion, Backrest Passad2 520X550, Plush Grey		1
15	1429278	Cushion, Backrest Passad2 580X550, Dartex Grey		1
15	1444331	Cushion, Backrest Passad2 580X550, Plush Grey		1
16	1444275	Cushion, Backrest Shoulder High 440X650, Dartex Grey		1

Cushions, Flex2 backrest =>2012-04-01


Pos.	Part number	Description	Valid from → until	Qty
17	1444276	Cushion, Backrest Shoulder High 470X650, Dartex Grey		1
18	1444277	Cushion, Backrest Shoulder High 520X650, Dartex Grey		1
19	1444278	Cushion, Backrest Shoulder High 580X650, Dartex Grey		1
20	1435592	BACKREST CUSHION 370X540, VICAIR MF, DARTEX GREY		1
21	1445741	BACKREST CUSHION 400X540, VICAIR MF, DARTEX GREY		1
22	1445742	BACKREST CUSHION 450X540, VICAIR MF, DARTEX GREY		1
23	1445743	BACKREST CUSHION 500X540, VICAIR MF, DARTEX GREY		1
24	1508865	Cover SW39, dartex grey		1
24	1444344	Cover SW39 TR32		1
25	1508866	Cover SW44, dartex grey		1
25	1444345	Cover SW44 TR32		1
26	1508867	Cover SW49, dartex grey		1
26	1444346	Cover SW49 TR32		1
27	1508868	Cover, Lateral 600X460, Dartex Grey		1
27	1444347	Cover, Lateral 600X460, Plush Grey Prima		1
28	1526459	Backrest upholstery SW 55 cm		1
28	1526573	Cover, Lateral 640X500, Plush Grey Prima		1
29	1526424	Cover, Lateral 700X500, Dartex Grey		1
29	1526572	Cover, Lateral 700X500, Plush Grey Prima		1
30	1513037	WEDGES, PAIR		1

Curvature cushion

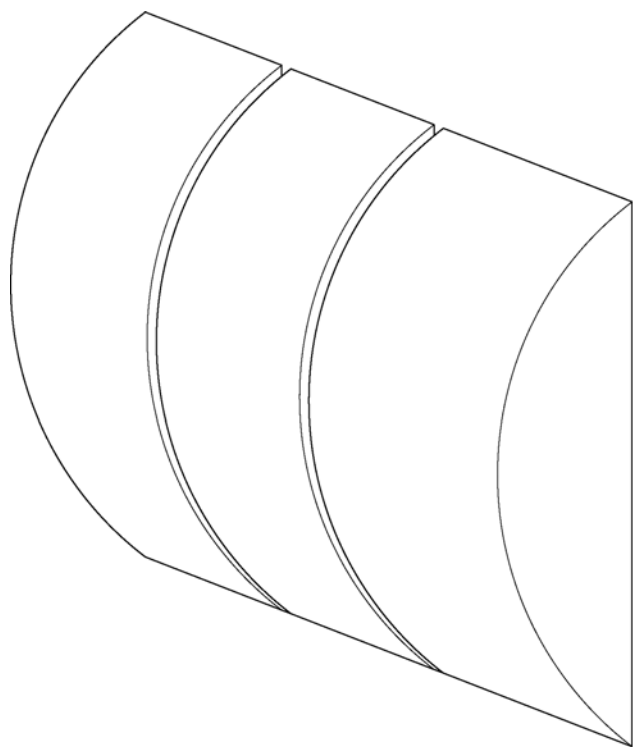

SPS5295081

Pos.	Part number	Description	Valid from → until	Qty
1	5252527	CURVATURE CUSHION F BACK		1

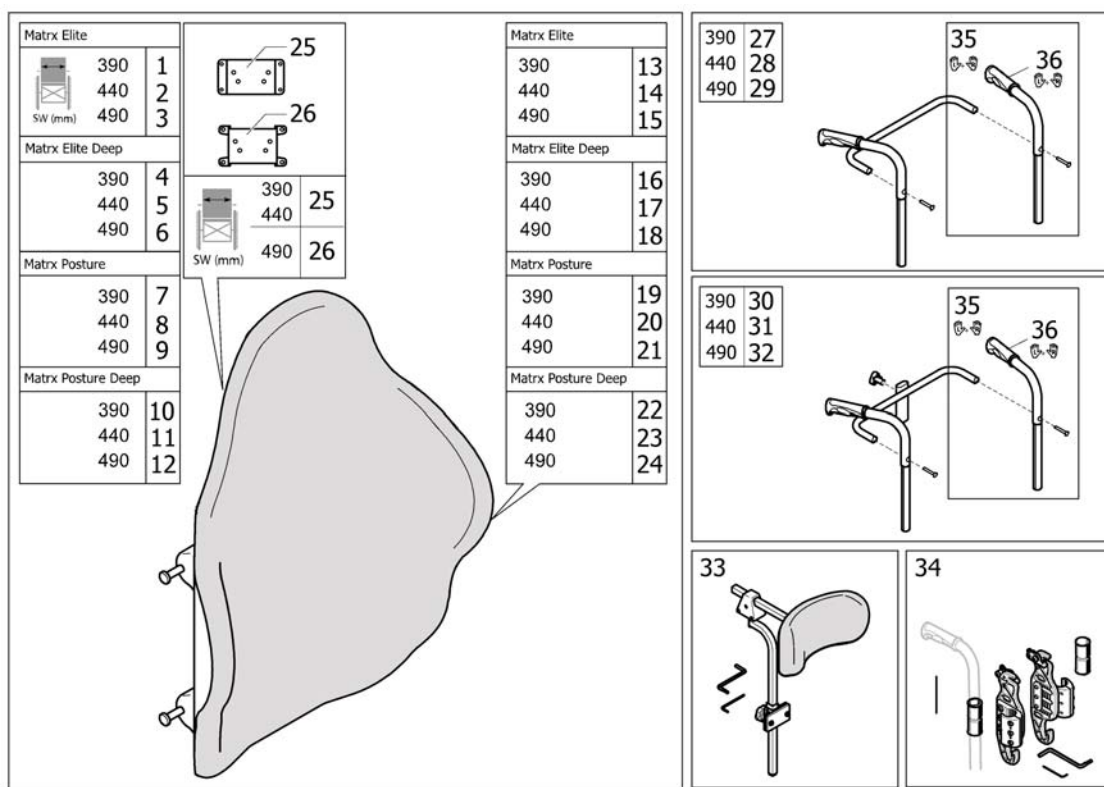
Backrest Matrix

Pos.	Part number	Description	Valid from → until	Qty
1	1532094	Backrest 380x505, Matrix Elite		1
2	1517755	Backrest 430x505, Matrix Elite		1
3	1517757	Backrest 480x505, Matrix Elite		1
4	1543511	Backrest 380x505, Matrix Elite Deep		1
5	1543519	Backrest 430x505, Matrix Elite Deep		1
6	1543987	Backrest 480x505, Matrix Elite Deep		1
7	1515728	Backrest 410x510, Matrix Posture Back		1
8	1515732	Backrest 460x510, Matrix Posture Back		1
9	1518132	Backrest 510x510, Matrix Posture Back		1
10	1545059	Backrest 410x510, Matrix Posture Back Deep		1
11	1545065	Backrest 460x510, Matrix Posture Back Deep		1
12	1545068	Backrest 510x510, Matrix Posture Back Deep		1
13	SP1545136	Cover 380x505, Elite		1
14	SP1545147	Cover 430x505, Elite		1
15	SP1545159	Cover 480x505, Elite		1
16	SP1544069	Cover 380x505, Elite Deep		1
17	SP1544077	Cover 430x505, Elite Deep		1
18	SP1544085	Cover 480x505, Elite Deep		1
19	SP1544897	Cover 405x505, Posture		1
20	SP1544996	Cover 455x505, Posture		1
21	SP1544999	Cover 480x505, Posture		1
22	SP1545073	Cover 405x505, Posture Deep		1
23	SP1545079	Cover 455x505, Posture Deep		1
24	SP1545082	Cover 505x505, Posture Deep		1
25	1544188	Elite Headrest Mounting Hardware - Medium		1
26	1544187	Elite Headrest Mounting Hardware - Large		1
27	SP1571966	PUSH HANDLES 390, FIXED CPL.		1
28	SP1571967	PUSH HANDLES 440, FIXED CPL.		1
29	SP1571968	PUSH HANDLES 490, FIXED CPL.		1
30	SP1572531	PUSH HANDLES 390, FIXED CPL. (FOR NECK- AND HEADREST)		1
31	SP1572574	PUSH HANDLES 440, FIXED CPL. (FOR NECK- AND HEADREST)		1

Backrest Matrix


Pos.	Part number	Description	Valid from → until	Qty
32	SP1572575	PUSH HANDLES 490, FIXED CPL. (FOR NECK- AND HEADREST)		1
33	1545274	Matrix 255 mm Soft Pad Headrest Kit (including fixation for PB and PB deep)		1
34	SP1545275	Mounting hardware, Matrix		1
35	SP1572734	PUSH HANDLE CPL., PAIR		1
36	SP1537634	HANDLE 22MM, PAIR		1

Backrest Matrix, Azalea Base

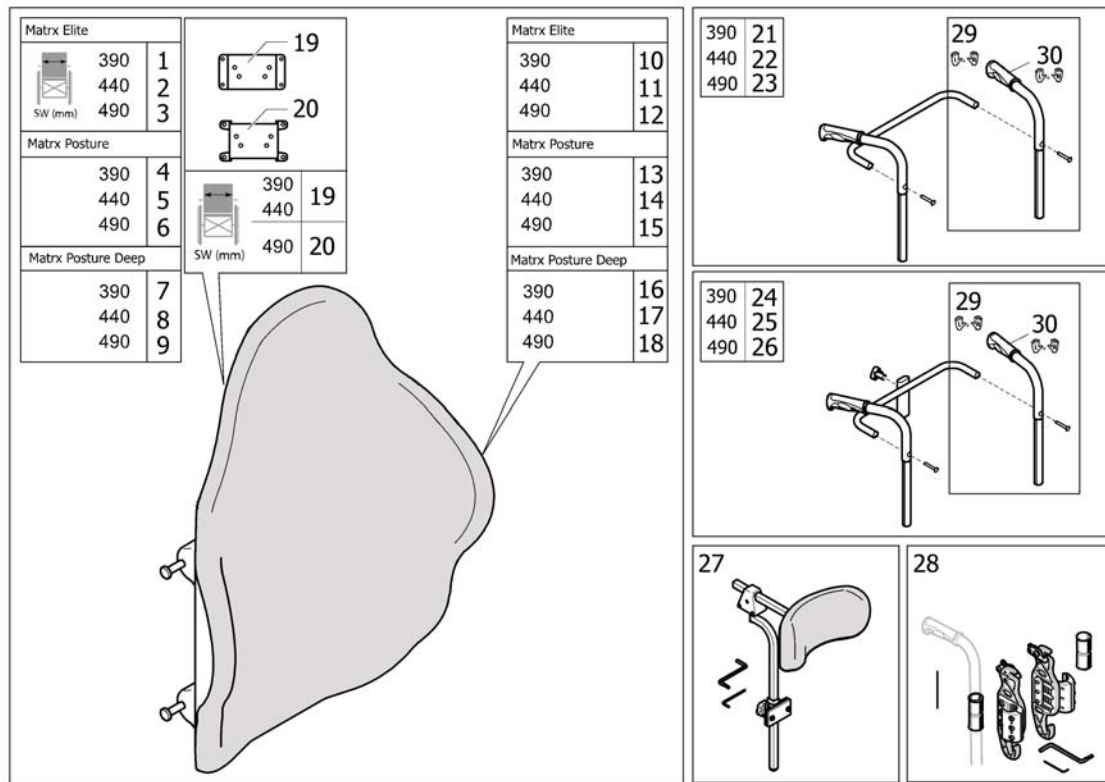


Pos.	Part number	Description	Valid from → until	Qty
1	1532094	Backrest 380x505, Matrix Elite		1
2	1517755	Backrest 430x505, Matrix Elite		1
3	1517757	Backrest 480x505, Matrix Elite		1
4	1515728	Backrest 410x510, Matrix Posture Back		1
5	1515732	Backrest 460x510, Matrix Posture Back		1
6	1518132	Backrest 510x510, Matrix Posture Back		1
7	1545059	Backrest 410x510, Matrix Posture Back Deep		1
8	1545065	Backrest 460x510, Matrix Posture Back Deep		1
9	1545068	Backrest 510x510, Matrix Posture Back Deep		1
10	SP1545136	Cover 380x505, Elite		1
11	SP1545147	Cover 430x505, Elite		1
12	SP1545159	Cover 480x505, Elite		1
13	SP1544897	Cover 405x505, Posture		1
14	SP1544996	Cover 455x505, Posture		1
15	SP1544999	Cover 480x505, Posture		1
16	SP1545073	Cover 405x505, Posture Deep		1
17	SP1545079	Cover 455x505, Posture Deep		1
18	SP1545082	Cover 505x505, Posture Deep		1
19	1544188	Elite Headrest Mounting Hardware - Medium		1
20	1544187	Elite Headrest Mounting Hardware - Large		1
21	SP1571966	PUSH HANDLES 390, FIXED CPL.		1
22	SP1571967	PUSH HANDLES 440, FIXED CPL.		1
23	SP1571968	PUSH HANDLES 490, FIXED CPL.		1
24	SP1572531	PUSH HANDLES 390, FIXED CPL. (FOR NECK- AND HEADREST)		1
25	SP1572574	PUSH HANDLES 440, FIXED CPL. (FOR NECK- AND HEADREST)		1
26	SP1572575	PUSH HANDLES 490, FIXED CPL. (FOR NECK- AND HEADREST)		1
27	1545274	Matrix 255 mm Soft Pad Headrest Kit (including fixation for PB and PB deep)		1
28	SP1545275	Mounting hardware, Matrix		1
29	SP1572734	PUSH HANDLE CPL., PAIR		1
30	SP1537634	HANDLE 22MM, PAIR		1

Legrest attachment

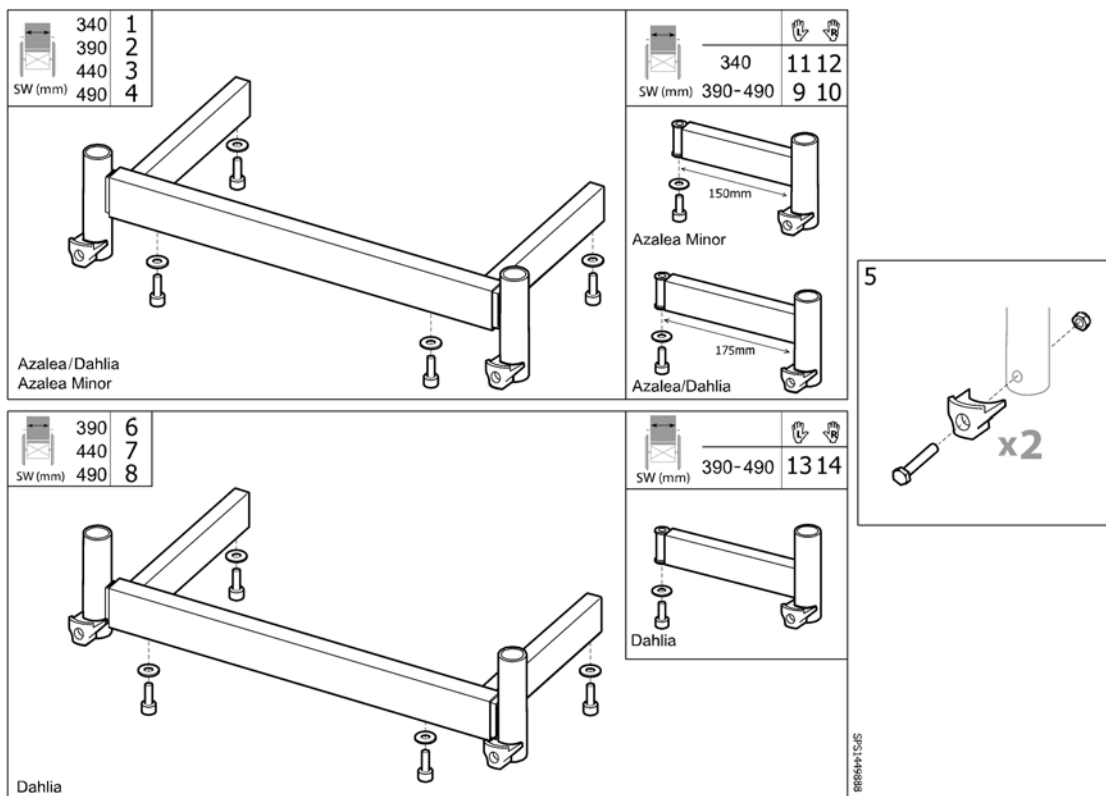


Pos.	Part number	Description	Valid from → until	Qty
1	1508926-0509	Hanger attachment 340 adj, cpl.		1
2	1449888-0509	Hanger attachment 390 adj, cpl.		1
3	1449889-0509	Hanger attachment 440 adj, cpl.		1
4	1449890-0509	Hanger attachment 490 adj, cpl.		1
5	1508091	KIT, LEGREST ATTACHM. BLOCK PAIR		1
6	SP1574112	Hanger attachment 390 adj, high, cpl.		1
7	SP1574113	Hanger attachment 440 adj, high, cpl.		1
8	SP1574114	Hanger attachment 490 adj, high, cpl.		1
9	SP1579059	Legrest support, LH, cpl.		1
10	SP1579440	Legrest support, RH, cpl.		1
11	SP1513015	Legrest support, LH, Azalea Minor, cpl.		1
12	SP1513016	Legrest support, RH, Azalea Minor, cpl.		1
13	SP1574125	Legrest support, high, LH, cpl.		1
14	SP1574124	Legrest support, high, RH, cpl.		1

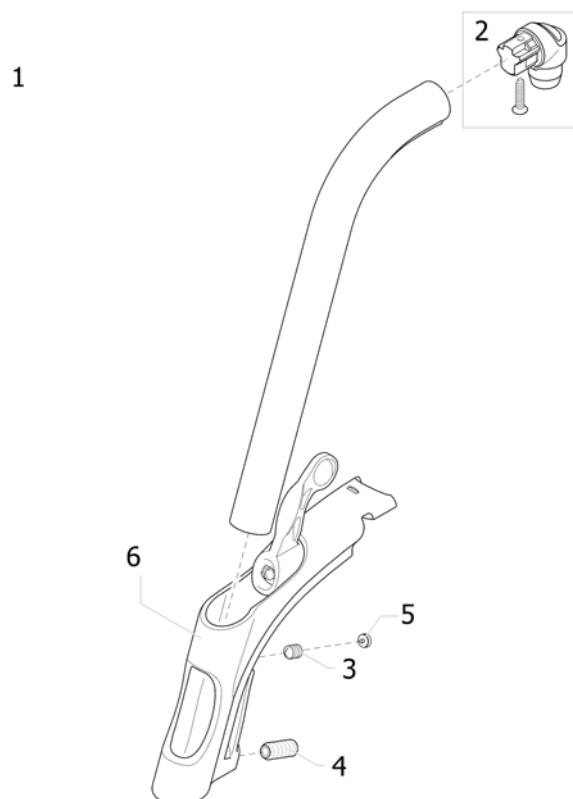Legrest 80° Alu2003, Fixed

Pos.	Part number	Description	Valid from → until	Qty
1	1493055-0509	Hanger Alu2003,Fixed 80 Deg,Cpl		1
2	1430426	Hanger pivot kit		1
3	1424572	Screw stop M10x12		1
4	1424571	Screw stop M10x23		1
5	1435741	Footrest Hanger Plug		1
6	1434362	Hanger support complete (Alu Pin)		1

Legrest 90° Alu2003, Fixed

1



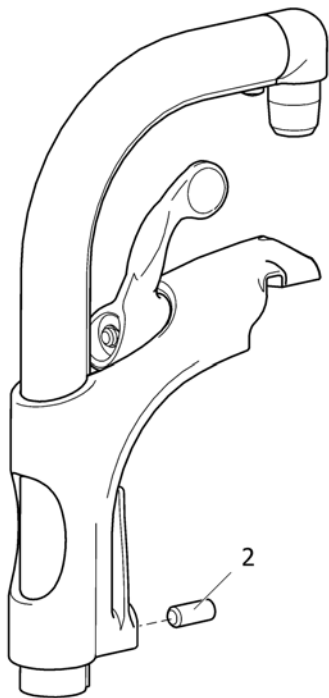


SPS1403256

Pos.	Part number	Description	Valid from → until	Qty
1	1493056-0509	Hanger fixed 90°		1
2	1424571	Screw stop M10x23		1


Legrest AA, ALU2008

	SW (mm)	L	R
Azalea, Clematis, Dahlia	340-390	1	2
	440	3	4
	490	5	6
Bellis, Bellis 150	390-430 (S)	3	4
	440-480 (M)	5	6
	490-530 (L)	5	6

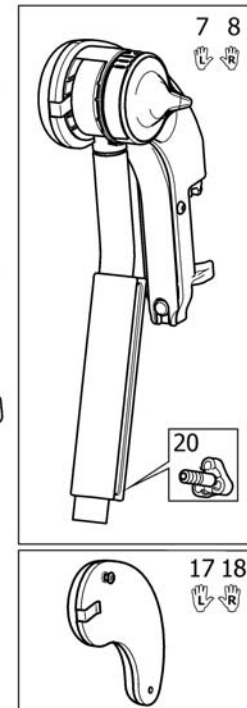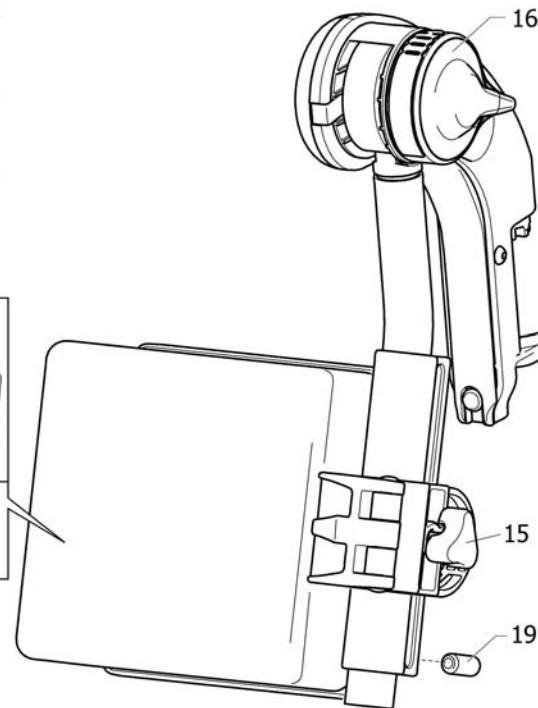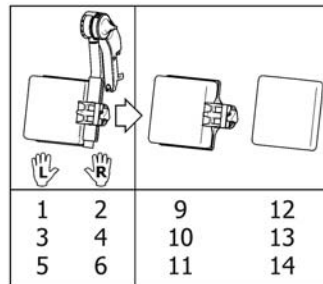

Pos.	Part number	Description	Valid from → until	Qty
1	1524984	HANGER (+ CALFPAD 12), LE, AA ALU2008 LEGREST		1
2	1524985	HANGER (+ CALFPAD 12), RI, AA ALU2008 LEGREST		1
3	1524986	HANGER (+ CALFPAD 15), LE, AA ALU2008 LEGREST		1
4	1524987	HANGER (+ CALFPAD 15), RI, AA ALU2008 LEGREST		1
5	1524988	HANGER (+ CALFPAD 18), LE, AA ALU2008 LEGREST		1
6	1524989	HANGER (+ CALFPAD 18), RI, AA ALU2008 LEGREST		1
7	1513007	Lh elevating legrest with kneepad Dartex		1
8	1513006	Rh elevating legrest with kneepad Dartex		1
9	1516313	CALF PAD 12 (140x170x80), CPL.		1
10	1516314	Calfpad Cushion, 19", Dartex black		1
11	1516315	CALF PAD 18 (190x170x80), CPL		1
12	1445763	PAD, CALFSUPPORT 12 CPL, DARTEX BLACK		1
13	1445764	PAD, CALFSUPPORT 15 CPL, DARTEX BLACK		1
14	1445765	PAD, CALFSUPPORT 18 CPL, DARTEX BLACK		1
15	1429215	HANDWHEEL M8		1
16	1508943	Incline adjustment lever		1
17	1513671	KNEEPAD (INCL SCREW), LE, AA LEGREST ALU2008	2008-10-01 →	1
18	1513716	KNEEPAD (INCL SCREW), RI, AA LEGREST ALU2008	2008-10-01 →	1
19	1424571	Screw stop M10x23		1
20	1429326	Handwheel M10, Spring		1


Legrest AA, Azalea Max

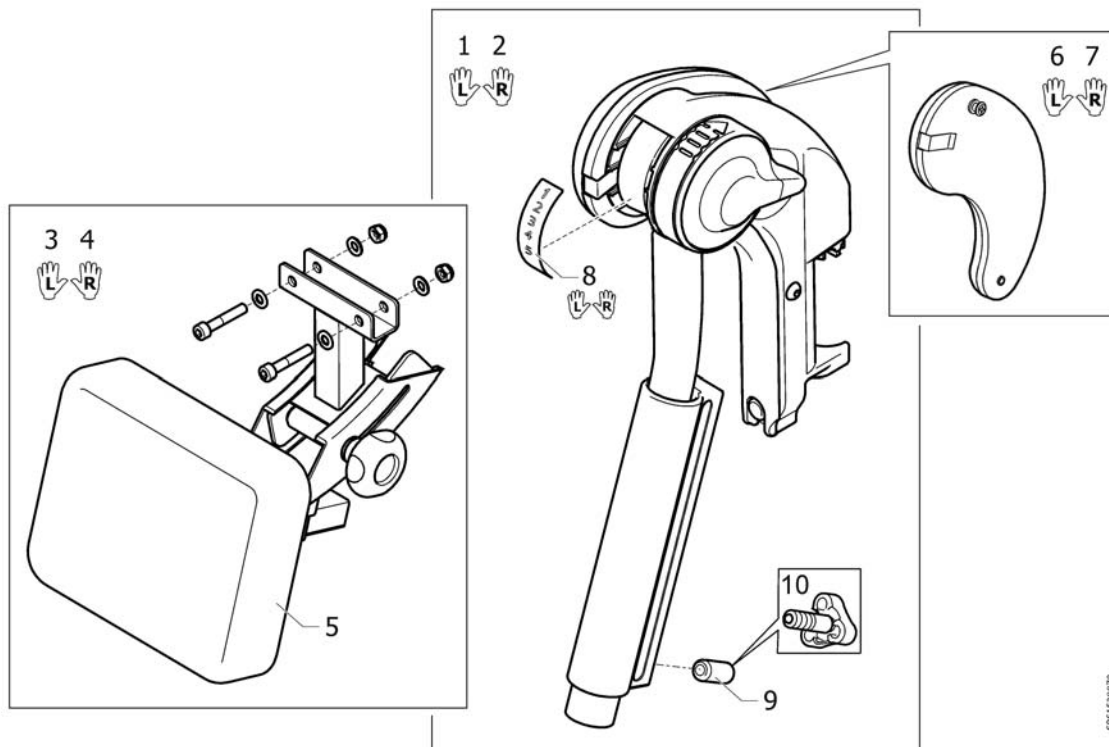

SP51528870

Pos.	Part number	Description	Valid from → until	Qty
1	1528870	Hanger AA, Left, Cpl, Azalea Max		1
2	1528871	Hanger AA, Right, Cpl, Azalea Max		1
3	1525057	Calf Pad Cpl, Le, Azalea Max		1
4	1525058	Calf Pad Cpl, Ri, Azalea Max		1
5	1525059	Pad Cpl, Calf Pad, Azalea Max		1
6	1513671	KNEEPAD (INCL SCREW), LE, AA LEGREST ALU2008		1
7	1513716	KNEEPAD (INCL SCREW), RI, AA LEGREST ALU2008		1
8	1535458	Sticker Kit, Hanger Aa, Azalea Max		1
9	1424571	Screw stop M10x23		1
10	1429326	Handwheel M10, Spring		1

1 2
👤 👤

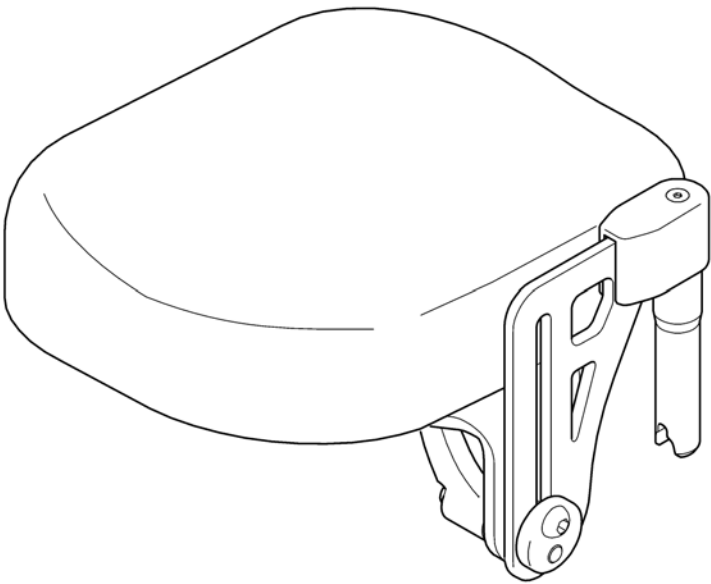


0002411506

Pos.	Part number	Description	Valid from → until	Qty
1	1442991	Support Lh, legrest amputee		1
2	1442990	Support Rh, legrest amputee		1

Amputee legrest, Azalea Max


1

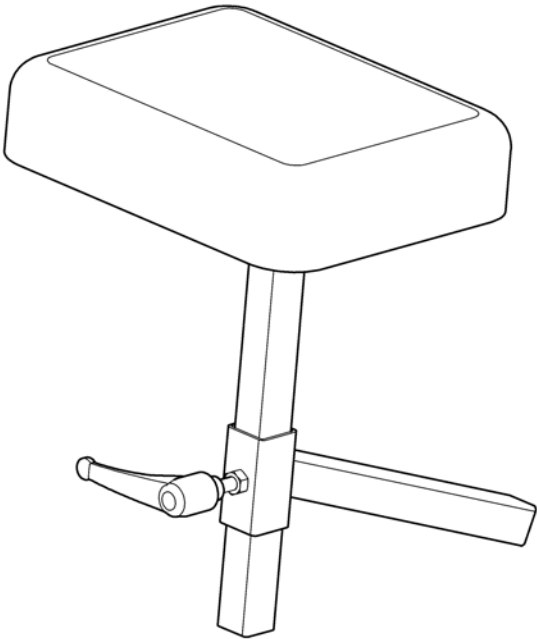

SPS1563433


Pos.	Part number	Description	Valid from → until	Qty
1	SP1563433	Amputee legrest, Azalea Max		1

Footrest, fixed, Alu2003

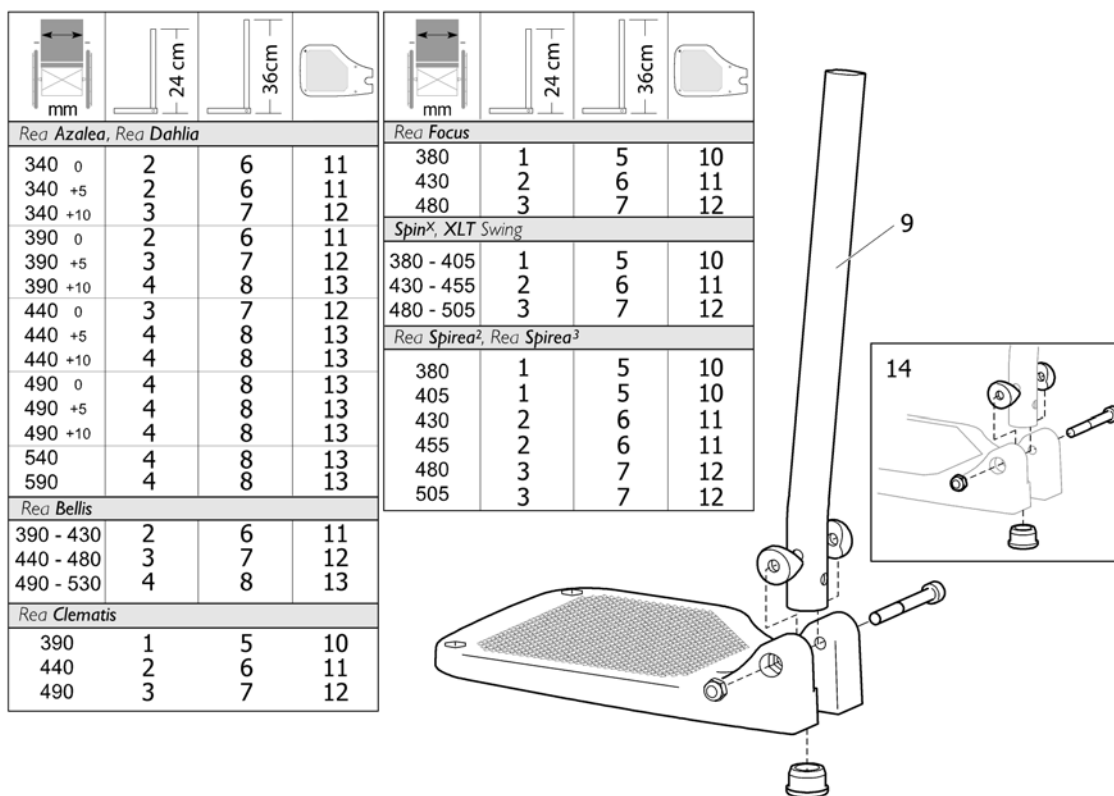

Pos.	Part number	Description	Valid from → until	Qty
1	1428357	FOOTREST 19, "ALU2003", FIXED, SHORT, LEFT		1
1	1428358	FOOTREST 19, "ALU2003", FIXED, SHORT, RIGHT		1
2	1428361	FOOTREST 21ALU2003,LOWER PART,FIXED,SHORT,LEFT		1
2	1428362	FOOTREST 21ALU2003,LOWER PART,FIXED,SHORT,RIGHT		1
3	1428365	FOOTREST 24ALU2003,LOWER PART,FIXED,SHORT,LEFT		1
3	1428366	FOOTREST 24ALU2003,LOWER PART,FIXED,SHORT,RIGHT		1
4	1435756	FOOTREST 27ALU2003,LOWER PART,FIXED,SHORT,LEFT		1
4	1435757	FOOTREST 27ALU2003,LOWER PART,FIXED,SHORT,RIGHT		1
5	1428359	FOOTREST 19, "ALU2003", FIXED, LONG, LEFT		1
5	1428360	FOOTREST 19, "ALU2003", FIXED, LONG, RIGHT		1
6	1428363	LOWER FOOTREST STANDARD WITH HEELSTRAP, width 43-45,5cm, LEFT		1
6	1428364	LOWER FOOTREST STANDARD, WITH HEELSTRAP, width 43-45,5cm, RIGHT		1
7	1428367	Footrest 24 Fixed, Long, Lh, Alu2003		1
7	1428368	Footrest 24 Fixed, Long, Rh, Alu2003		1
8	1435659	FOOTREST 27ALU2003,LOWER PART,FIXED,LONG,LEFT		1
8	1435660	FOOTREST 27ALU2003,LOWER PART,FIXED,LONG,RIGHT		1
9	1434561-0010	Standard tube for fixed footrest		1
9	1434563-0010	Long tube for fixed and a&d adjustable footrest		1
10	1428340	Fixed footplate 175mm (SW380-405) Left		1
10	1428341	Fixed footplate 175mm (SW380-405) Right		1
11	1424574	Fixed footplate 200mm (SW430-455) Left		1
11	1424575	Fixed footplate 200mm (SW430-455) Right		1
12	1428346	Fixed footplate 225mm (SW480-505) Left		1
12	1428347	Fixed footplate 225mm (SW480-505) Right		1
13	1428374	Fixed footplate NG Lh 250 mm (SW550/605 Action NG)(SW490 Esprit Clematis)		1
13	1428375	Fixed footplate NG Rh 250 mm (SW550/605 Action NG)(SW490 Esprit Clematis)		1
14	1430433	Kit screws for fixed footrest		1

Footrest, AA, Alu2003


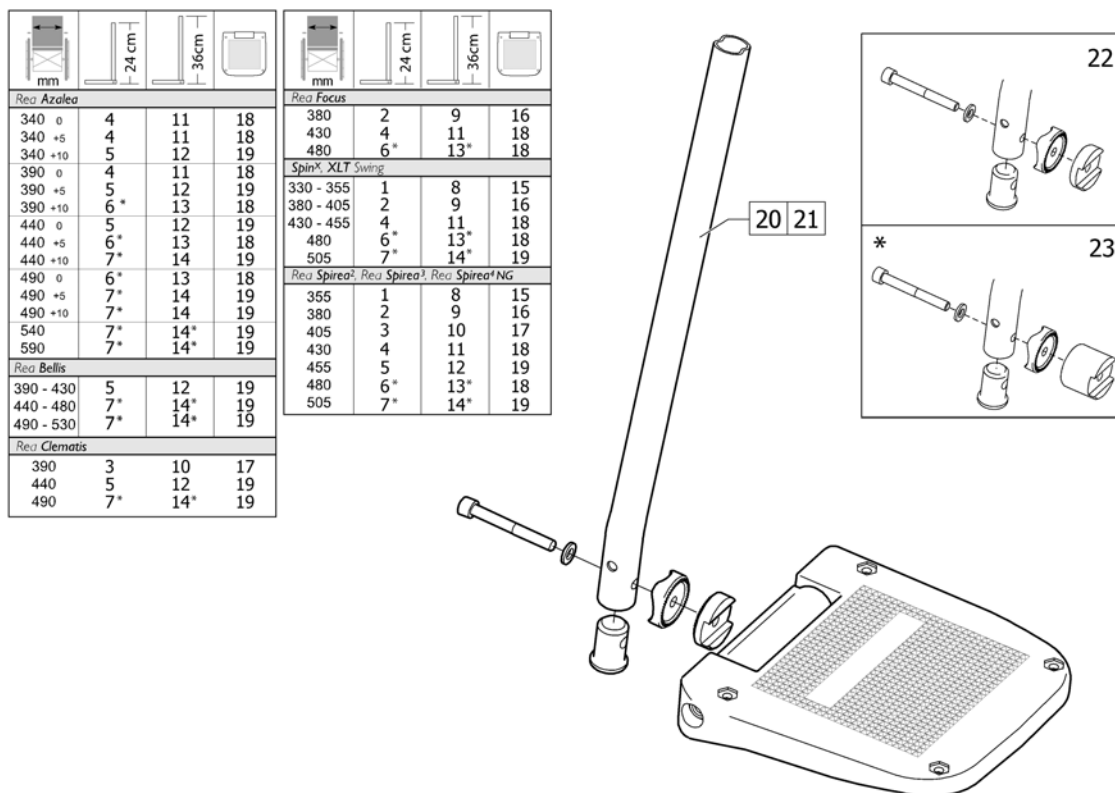




Pos.	Part number	Description	Valid from → until	Qty
1	1485430	Footrest 12 Short, Le, "Alu2003"		1
1	1485524	Footrest 12 Short, Ri, "Alu2003"		1
2	1429159	Footrest 14 aa, "Alu2003", Short, Left		1
2	1429160	Footrest 14 aa, "Alu2003", Short, Right		1
3	1429161	Footrest 16 aa, "Alu2003", Short, Left		1
3	1429162	Footrest 16 aa, "Alu2003", Short, Right		1
4	1429163	Footrest 17 aa, "Alu2003", Short, Left		1
4	1429164	Footrest 17 aa, "Alu2003", Short, Right		1
5	1429165	Footrest 18, "Alu2003", Lower Part, aa, Short, Left		1
5	1429166	Footrest 18, "Alu2003", Lower Part, aa, Short, Right		1
6	1429167	Footrest 19 aa, "Alu2003", Short, Left		1
6	1429168	Footrest 19 aa, "Alu2003", Short, Right		1
7	1429169	Footrest 20 Alu2003, Lower Part, aa, Short, Left		1
7	1429170	Footrest 20 Alu2003, Lower Part, aa, Short, Right		1
8	1485428	Footrest 12 Long, Le, "Alu2003"		1
8	1485429	Footrest 12 Long, Ri, "Alu2003"		1
9	1429171	Footrest 14 aa, "Alu2003", Long, Left		1
9	1429172	Footrest 14 aa, "Alu2003", Long, Right		1
10	1429173	Footrest 16 aa, "Alu2003", Long, Left		1
10	1429174	Footrest 16 aa, "Alu2003", Long, Right		1
11	1429175	Footrest 17 aa, "Alu2003", Long, Left		1
11	1429176	Footrest 17 aa, "Alu2003", Long, Right		1
12	1429177	Footrest 18, "Alu2003", Lower Part, aa, Long, Left		1
12	1429178	Footrest 18, "Alu2003", Lower Part, aa, Long, Ri		1
13	1429179	Footrest 19 aa, "Alu2003", Long, Left		1
13	1429180	Footrest 19 aa, "Alu2003", Long, Right		1
14	1429181	Footrest 20 Alu2003, Lower Part, aa, Long, Left		1
14	1429182	Footrest 20 Alu2003, Lower Part, aa, Long, Right		1
15	5106372	Footplate 12 aa, Cpl.		1
16	5106374	Footplate, angle and depth adjustable, 141 mm		1
17	5106375	Footplate, angle and depth adjustable, 156 mm		1

Footrest, AA, Alu2003

Pos.	Part number	Description	Valid from → until	Qty
18	5106594	Footplate, angle and depth adjustable, 166 mm		1
19	5106376	Footplate, angle and depth adjustable, 176 mm		1
20	1434561-0010	Standard tube for fixed footrest		1
21	1434563-0010	Long tube for fixed and a&d adjustable footrest		1
22	1430839	Kit fixation, adjustable footplate, short spacer		1
23	1430840	Kit fixation, adjustable footplate, long spacer		1

Footrest 90°, Alu2003

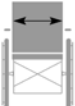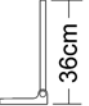
			
<i>Rea Bellis</i>			
S (39 - 43)	2	6	11
M (44 - 48)	3	7	12
L (49 - 53)	4	8	13
<i>Rea Azalea, Rea Comfort</i>			
34-39	2	6	11
44	3	7	12
49	4	8	13
<i>Rea Clematis</i>			
39	1	5	10
44	2	6	11
49	3	7	12
<i>Rea Focus</i>			
38	1	5	10
43	2	6	11
48	3	7	12
<i>SpinX, XLT Swing, Rea Spirea²</i>			
38 - 40,5	1	5	10
43 - 45,5	2	6	11
48 - 50,5	3	7	12

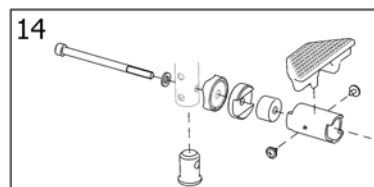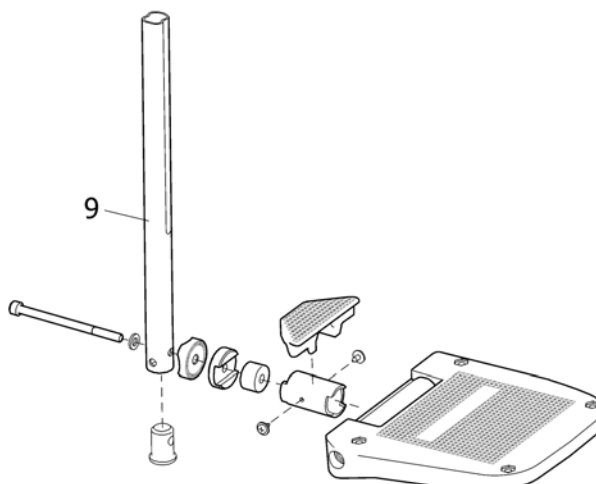

SPS1446130

Pos.	Part number	Description	Valid from → until	Qty
1	1446130	FOOTREST 11 AA, 90° "ALU2003", SHORT, LEFT		1
1	1446131	FOOTREST 11 AA, 90° "ALU2003", SHORT, RIGHT		1
2	1446132	FOOTREST LOWER PART 13, AA, FOOTR. 90, SHORT, LH		1
2	1446133	FOOTREST LOWER PART 13, AA, FOOTR. 90, SHORT, RH		1
3	1446134	FOOTREST LOWER PART 16, AA, FOOTR. 90, SHORT, LH		1
3	1446135	FOOTREST LOWER PART 16, AA, FOOTR. 90, SHORT, RH		1
4	1446159	FOOTREST LOWER PART 18, AA, FOOTR. 90, SHORT, LH		1
4	1446160	FOOTREST LOWER PART 18, AA, FOOTR. 90, SHORT, RH		1
5	1446136	FOOTREST 11 AA, 90° "ALU2003", LONG, LEFT		1
5	1446137	FOOTREST 11 AA, 90° "ALU2003", LONG, RIGHT		1
6	1446138	FOOTREST LOWER PART 13, AA, FOOTR. 90, LONG, LH		1
6	1446139	FOOTREST LOWER PART 13, AA, FOOTR. 90, LONG, RH		1
7	1446140	FOOTREST LOWER PART 16, AA, FOOTR. 90, LONG, LH		1
7	1446141	FOOTREST LOWER PART 16, AA, FOOTR. 90, LONG, RH		1
8	1446161	FOOTREST LOWER PART 18, AA, FOOTR. 90, LONG, LH		1
8	1446162	FOOTREST LOWER PART 18, AA, FOOTR. 90, LONG, RH		1
9	1440035	TUBE, LEGREST, LONG (346)		1
9	1446094	TUBE, LEGREST, SHORT (221)		1
10	5106371	Footplate, angle and depht adjustable, 106 mm		1
11	5106373	Footplate, angle and depth adjustable, 131 mm		1
12	5106375	Footplate, angle and depth adjustable, 156 mm		1
13	5106376	Footplate, angle and depth adjustable, 176 mm		1
14	1497913	SCREW KIT FOOTREST 90 ALU 2004		1

Footboard, Azalea Max


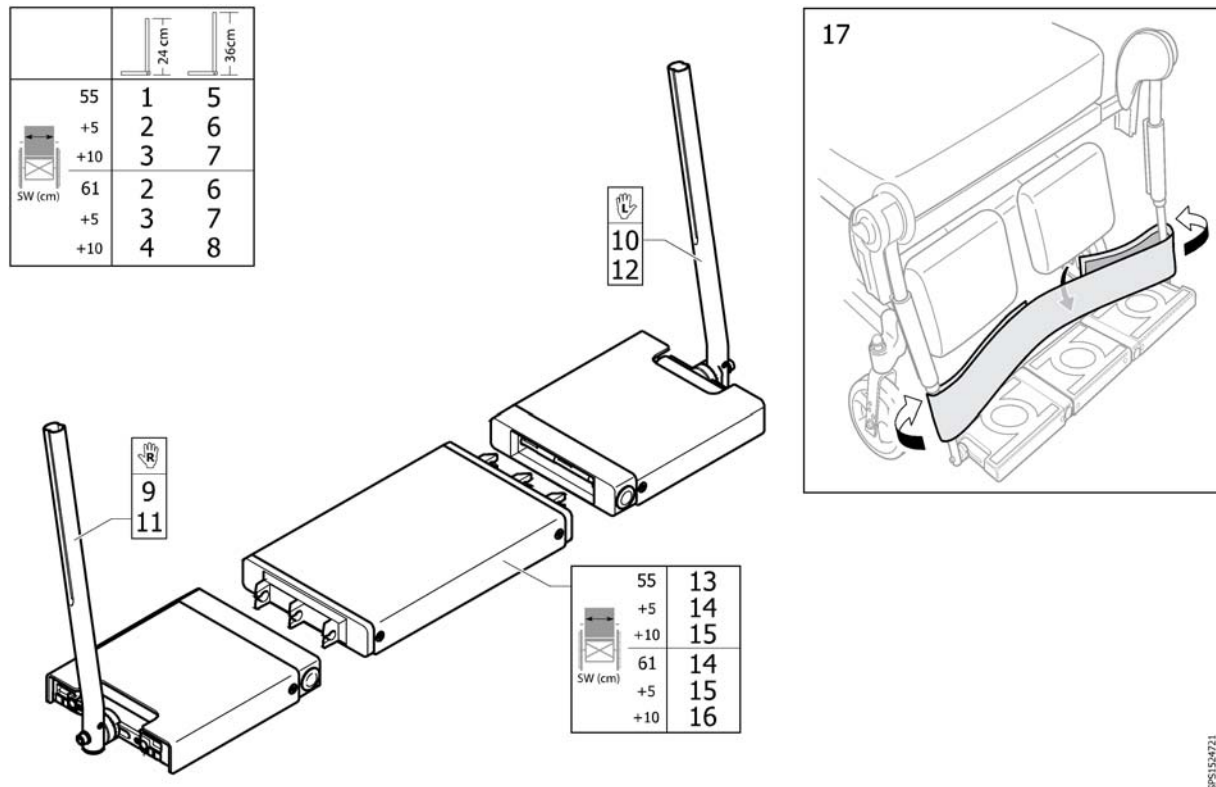


SPS1524721

Pos.	Part number	Description	Valid from → until	Qty
1	1524977	Footboard 55, Short Tubes		1
2	1524978	Footboard 61, Short Tubes		1
3	1524979	Footboard 65, Short Tubes		1
4	1524980	Footboard 71, Short Tubes		1
5	1524721	BOARD CONVERTER 55, CPL, LONG		1
6	1524722	BOARD CONVERTER 61, CPL, LONG		1
7	1524723	BOARD CONVERTER 65, CPL, LONG		1
8	1524724	BOARD CONVERTER 71, CPL, LONG		1
9	1524728	Right Part, Short Tube, Footboard		1
10	1524727	Left Part, Short Tube, Footboard		1
11	1524726	Right Part, Long Tube, Footboard		1
12	1524725	Left Part, Long Tube, Footboard		1
13	1524731	Center Part 55 (124), Footboard		1
14	1524732	Center Part 61 (184), Footboard		1
15	1524733	Center Part 65 (222), Footboard		1
16	1524734	Center Part 71 (282), Footboard		1
17	SP1532566	Heelstrap, Azalea Max		1

Central footrest

Rea Bellis	1
Rea Azalea Rea Dahlia	2

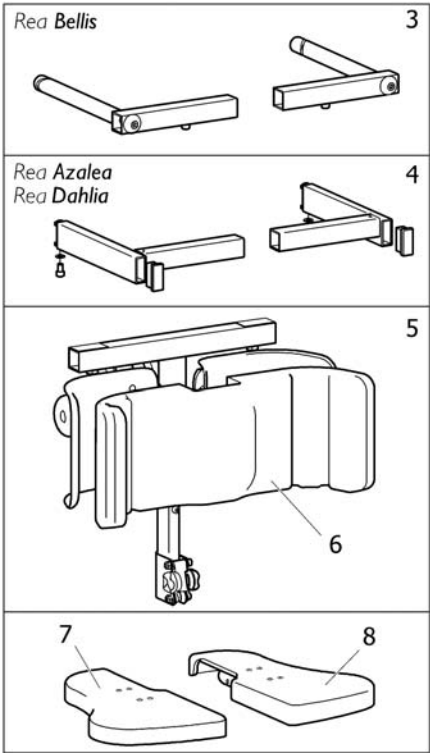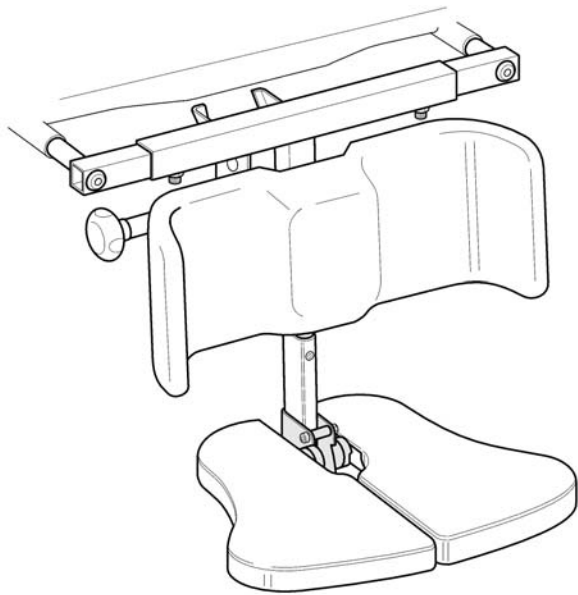


SPS1484821

Pos.	Part number	Description	Valid from → until	Qty
1	1424640	CENTRAL LEGREST CPL.		1
2	1484821	CENTER LEGREST CPL. (AZALEA)		1
3	1493102	ATTACHMENT CPL., CENTRAL LEGREST		1
4	1493118	ATTACHMENT CPL., CENTRAL LEGREST, AZALEA		1
5	1493101	POLE CPL, CENTRAL LEGREST		1
6	1424545	CALFPAD CUSION, CENTER LEGREST		1
7	1484006	FOOTPLATE CPL, RIGHT, CENTER FOOTREST		1
8	1484007	FOOTPLATE CPL, LEFT, CENTER FOOTREST		1


Calf strap, Alu2003 legrest

SPS1429214

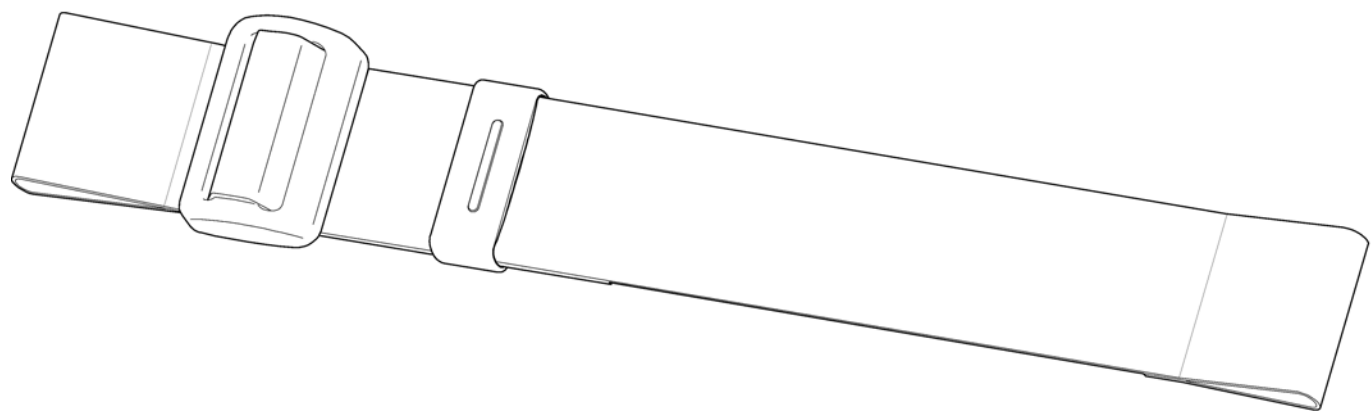

Pos.	Part number	Description	Valid from → until	Qty
	1429214	Calf strap adjustable, medium		1


1

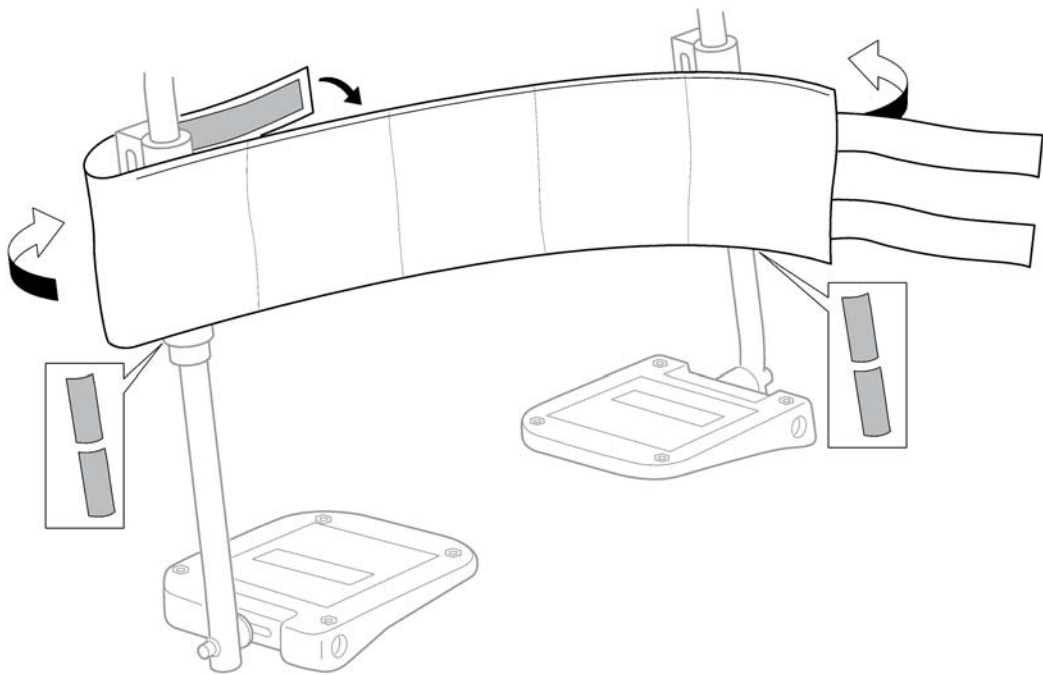

SPS1563951



Pos.	Part number	Description	Valid from → until	Qty
1	SP1563951	Calf strap, padded		1



Heel strap

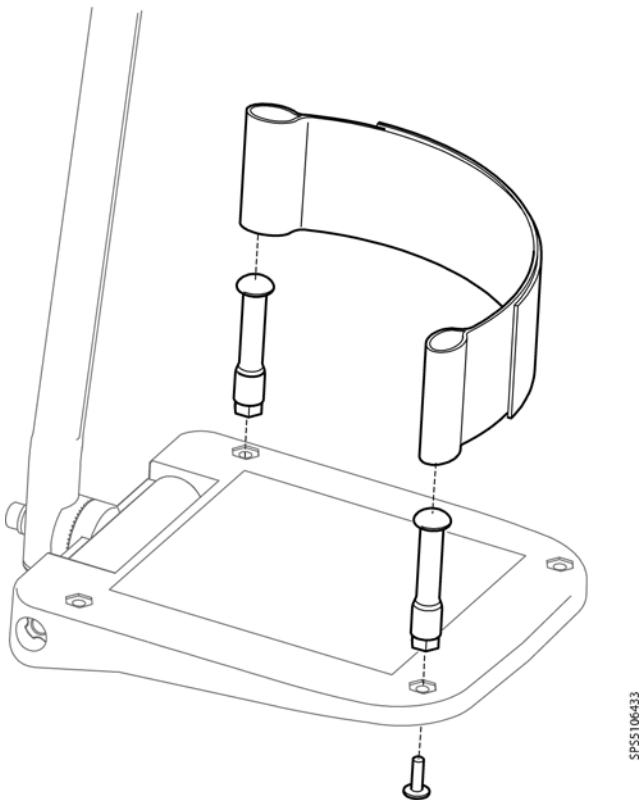

Pos.	Part number	Description	Valid from → until	Qty
	5321579	Heel strap 400/500 (pair)		1

Extender, footplate

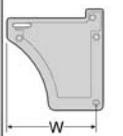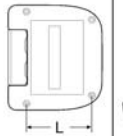		
135mm	99-119mm	1
160mm	124-144mm	2

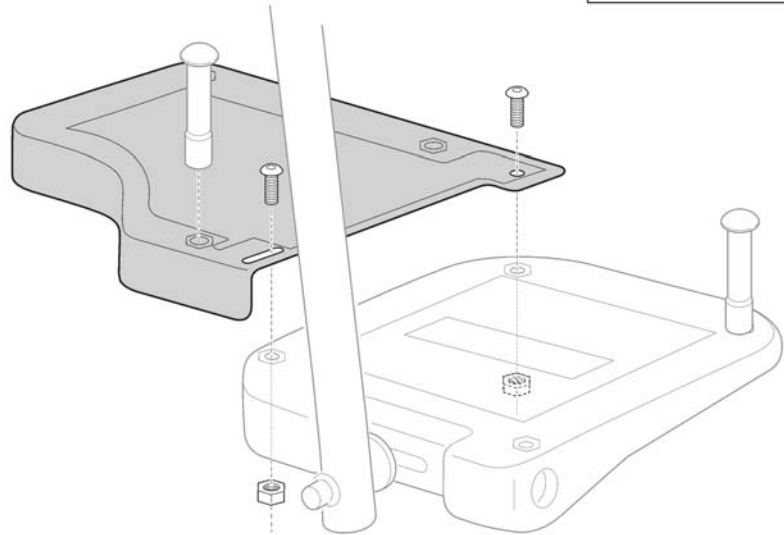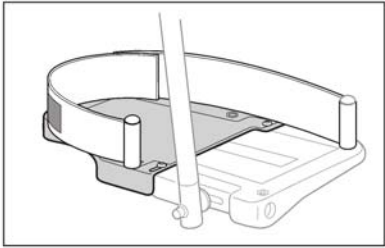



SP5255296

Pos.	Part number	Description	Valid from → until	Qty
1	SP5255297	Footplate extender w=135 pair		1
2	SP5255299	Footplate extender w=160 pair		1

Footrest one piece

SH (mm)	340			390			440			490		
	0	+50	+100	0	+50	+100	0	+50	+100	0	+50	+100
400-450	1	1	2	1	2	3	2	4	5	4	6	-
450-500	-	-	-	7	8	9	8	10	11	10	12	-

SW (mm)	390			440			490		
	0	+50	+100	0	+50	+100	0	+50	+100
SW (mm)	13	14	15	14	15	15	15	15	-

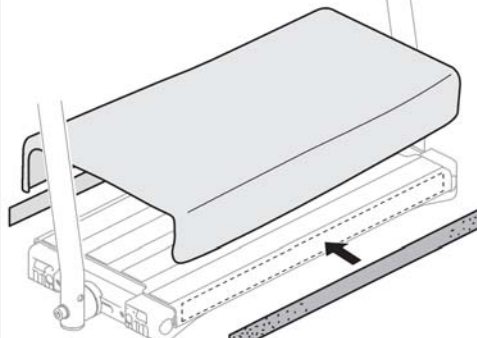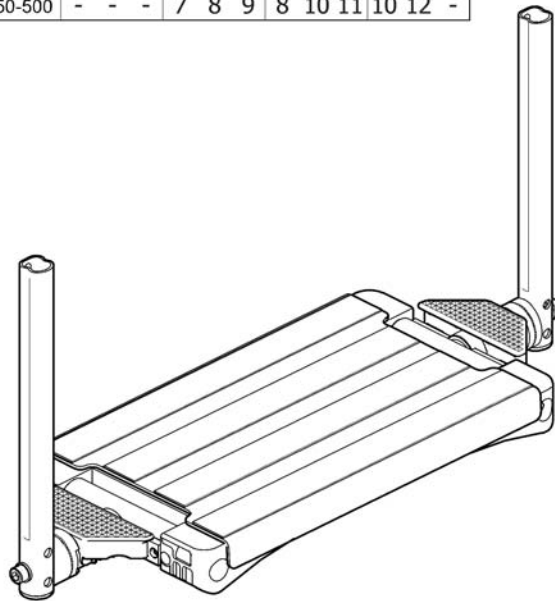



SP51493268

Pos.	Part number	Description	Valid from → until	Qty
1	1493268	FOOTREST ONE PIECE 272, ANGLE ADJ., SHORT		1
2	1493270	FOOTREST ONE PIECE 322, ANGLE ADJ., SHORT		1
3	1495821	FOOTREST ONE PIECE 347, ANGLE ADJ., SHORT		1
4	1495822	FOOTREST ONE PIECE 372, ANGLE ADJ., SHORT		1
5	1495823	FOOTREST ONE PIECE 397, ANGLE ADJ., SHORT		1
6	1495824	FOOTREST ONE PIECE 422, ANGLE ADJ., SHORT		1
7	1493358	FOOTREST ONE PIECE 272, ANGLE ADJ., LONG		1
8	1493377	FOOTREST ONE PIECE 322, ANGLE ADJ., LONG		1
9	1495825	FOOTREST ONE PIECE 347, ANGLE ADJ., LONG		1
10	1495826	FOOTREST ONE PIECE 372, ANGLE ADJ., LONG		1
11	1495827	FOOTREST ONE PIECE 397, ANGLE ADJ., LONG		1
12	1495828	FOOTREST ONE PIECE 422, ANGLE ADJ., LONG		1
13	SP1563959	COVER, ONE PIECE FOOTPLATE 390		1
14	SP1563962	COVER, ONE PIECE FOOTPLATE 440		1
15	SP1563963	COVER, ONE PIECE FOOTPLATE 490		1

Footplate cover

1

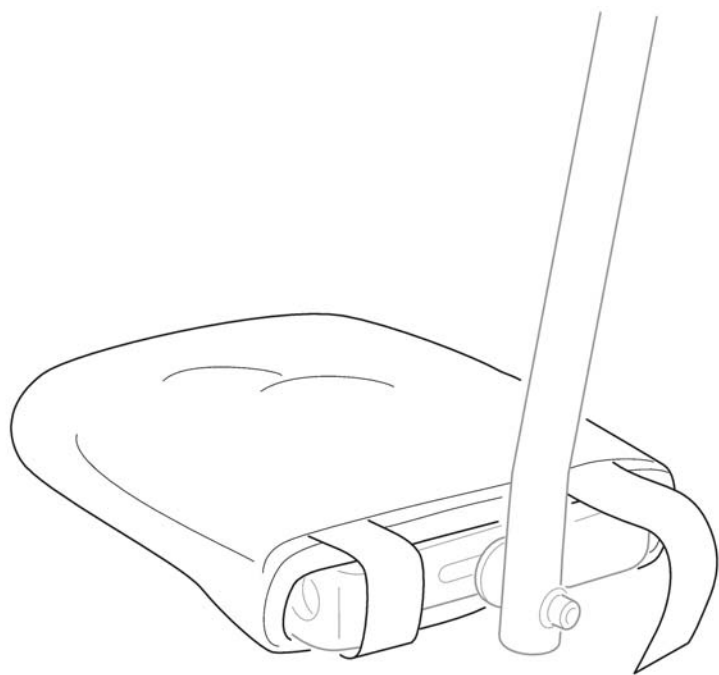


SP1563946

Pos.	Part number	Description	Valid from → until	Qty
1	SP1563946	COVER, FOOTPLATE		1

Footbox


390	1
440	2
490	3

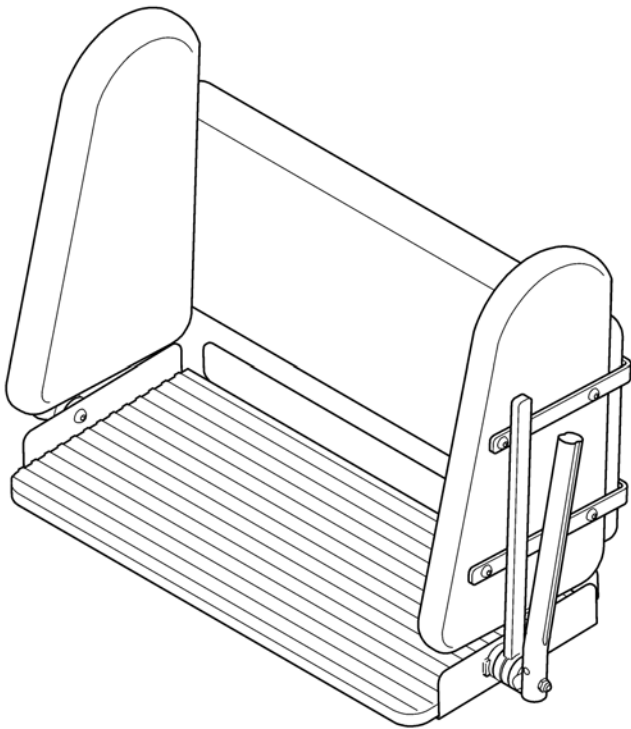

SP1571750


Pos.	Part number	Description	Valid from → until	Qty
1	SP1571749	FOOTBOX 390 CPL.		1
2	SP1571750	FOOTBOX 440 CPL.		1
3	SP1571751	FOOTBOX 490 CPL.		1

Armrest, Ducktail

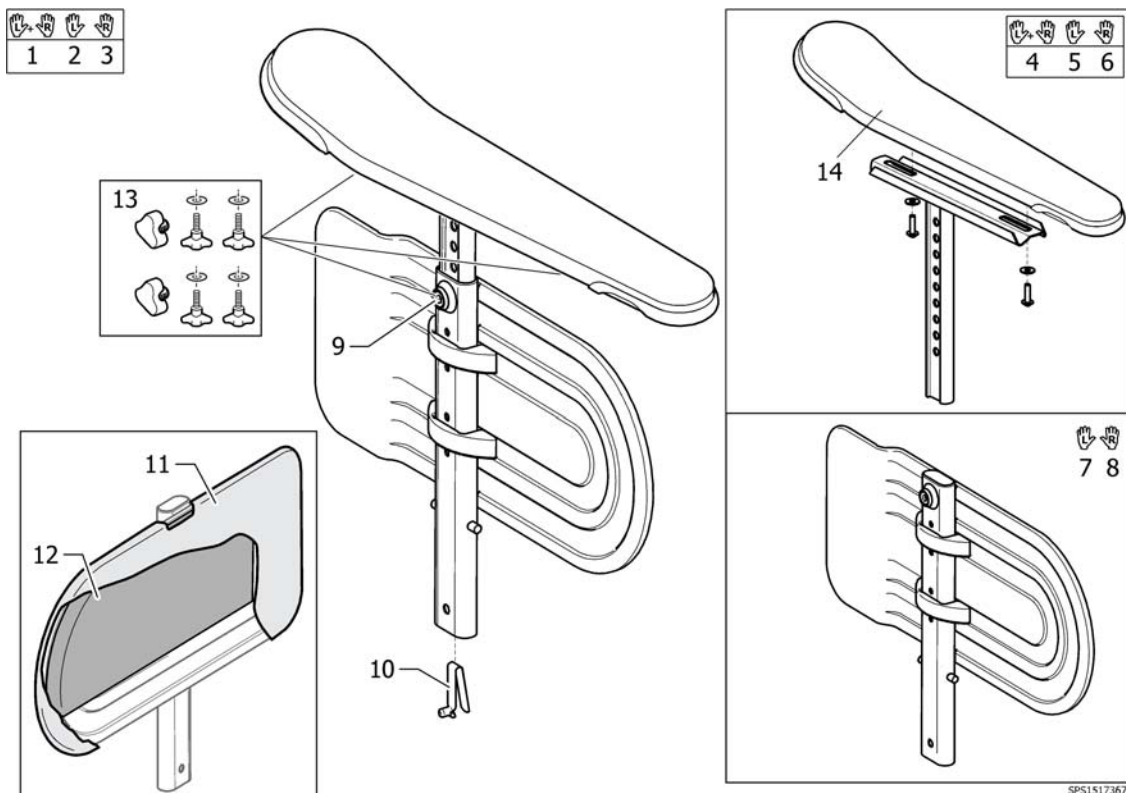




Pos.	Part number	Description	Valid from → until	Qty
1	1530007	ARMREST, HEIGHT ADJ., CPL. PAIR		1
2	1517368	ARMREST, HEIGH ADJ. LEFT CPL.		1
3	1517367	ARMREST, HEIGH ADJ. RIGHT CPL.		1
4	1518704	ARMREST UPPER PART, HIGHT ADJ. PAIR		1
5	1518702	Ampad with support adjustable in height left		1
6	1518701	Ampad with support adjustable in height right		1
7	1415521	Skirtguard LH, cpl.		1
8	1415520	Skirtguard RH, cpl.		1
9	5106533	Locking screw M12X10		1
10	1560734	Spring pin locked armrest		1
11	1415453	Cover protects clothing Dartex Black		1
12	SP1563199	Padding, pair, armrest		1
13	SP1567515	HANDWHEEL KIT, PAIR, ARMREST		1
14	SP1451561	Ampad, Ducktail		1

Armrest Low, Ducktail

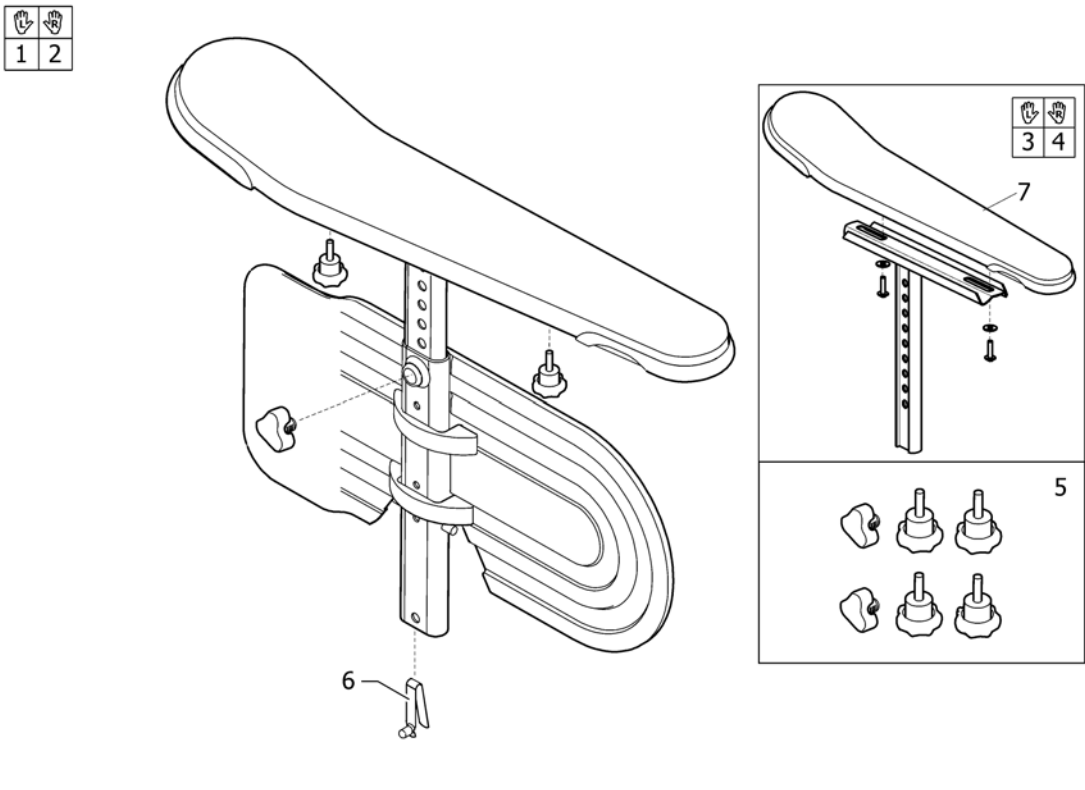


SPS1563965

Pos.	Part number	Description	Valid from → until	Qty
1	SP1563966	Armrest low, height adj., left		1
2	SP1563965	Armrest low, height adj., right		1
3	1518702	Ampad with support adjustable in height left		1
4	1518701	Ampad with support adjustable in height right		1
5	SP1567515	HANDWHEEL KIT, PAIR, ARMREST		1
6	1560734	Spring pin locked armrest		1
7	SP1451561	Ampad, Ducktail		1

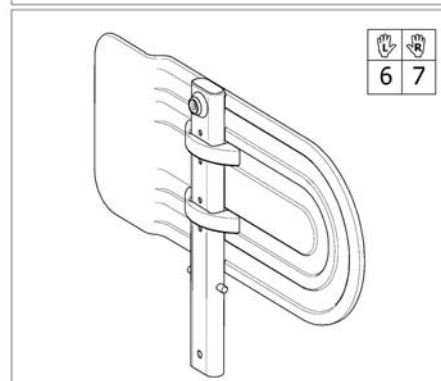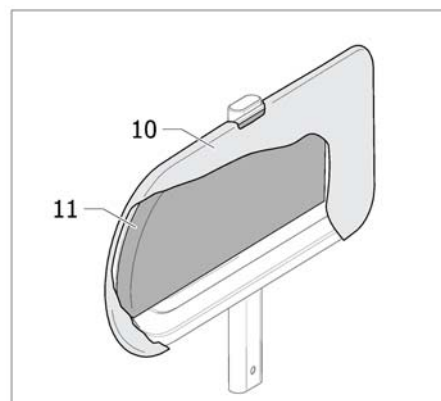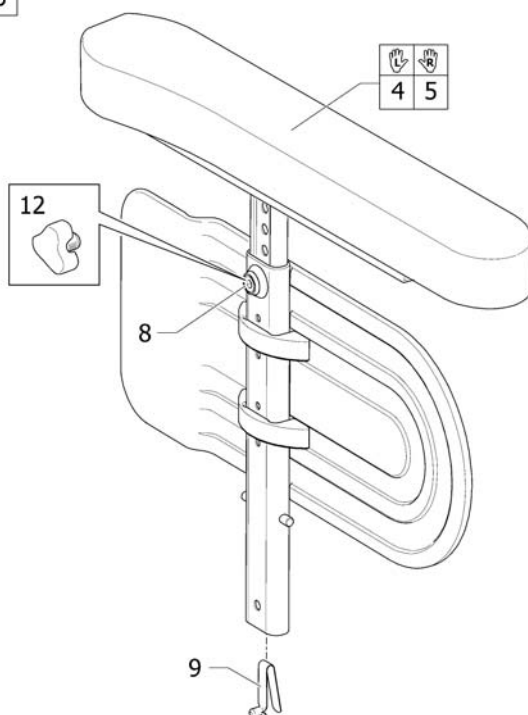Armrest, height adj.

SPS1427166


Pos.	Part number	Description	Valid from → until	Qty
1	1427166	ARMREST PAIR, HA		1
2	1427167	ARMREST LE ASSIST, CPL		1
3	1427168	ARMREST RI ASSIST, CPL		1
4	1427169	ARMREST U-PART CPL,LE,AS		1
5	1427170	ARMREST U-PART CPL,RI,AS		1
6	1415521	Skirtguard LH, cpl.		1
7	1415520	Skirtguard RH, cpl.		1
8	5106533	Locking screw M12X10		1
9	1560734	Spring pin locked armrest		1
10	1415453	Cover protects clothing Dartex Black		1
11	SP1563199	Padding, pair, armrest		1
12	5221122	Handwheel M12		1

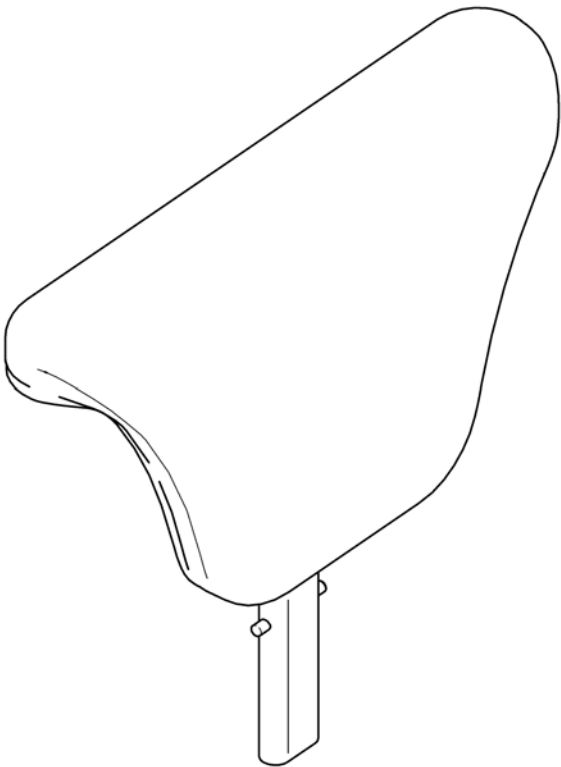

SP51492547


Pos.	Part number	Description	Valid from → until	Qty
	1492547	ARMREST, PADDED		1

Armrest Max, reinforced

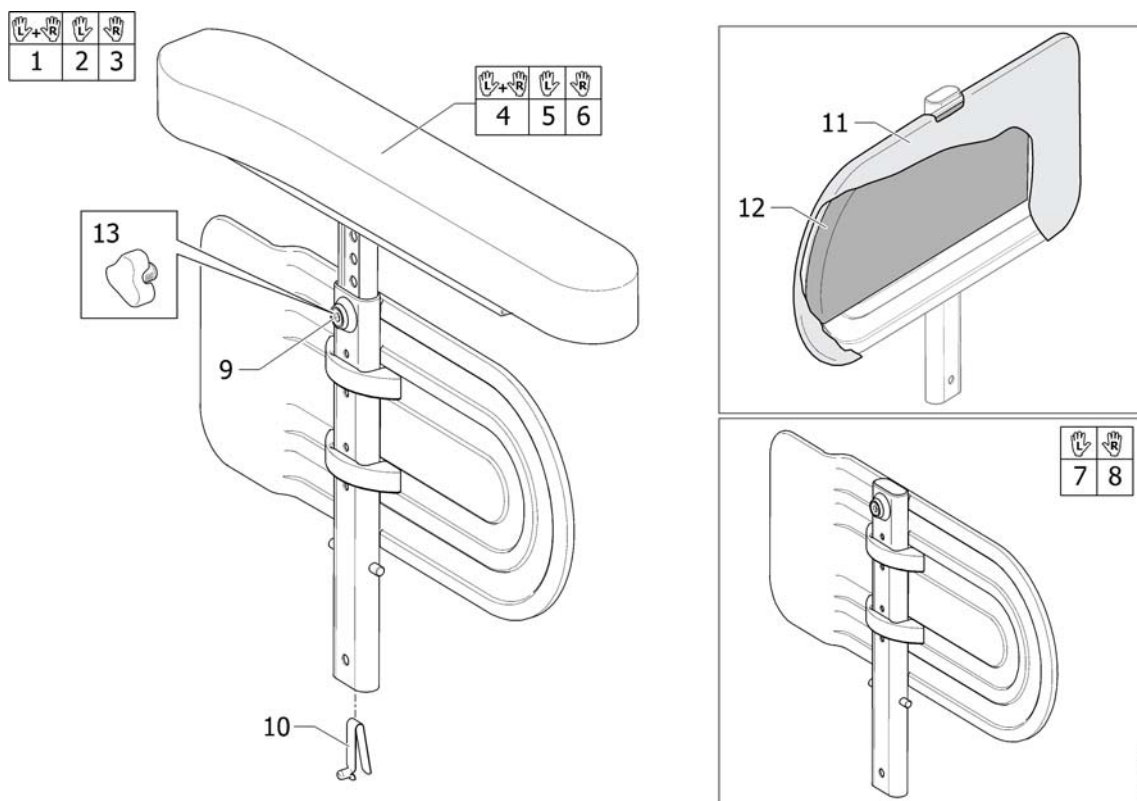


Pos.	Part number	Description	Valid from → until	Qty
1	1535270	Armrest Height Adj., Wide Pad, Pair		1
2	1525069	Armrest Height Adj., Wide Pad, Le		1
3	1525068	Armrest Height Adj., Wide Pad, Ri		1
4	1535271	Upper Part, Pair, Armrest Height Adj., Wide Pad		1
5	1525067	Upper part, Armrest Height Adj., Le		1
6	1525066	Upper part, Armrest Height Adj., Ri		1
7	1525048	Side Guard, Reinforced, Le, Cpl		1
8	1525049	Side Guard, Reinforced, Ri, Cpl		1
9	5106533	Locking screw M12X10		1
10	1560734	Spring pin locked armrest		1
11	1415453	Cover protects clothing Dartex Black		1
12	SP1563199	Padding, pair, armrest		1
13	5221122	Handwheel M12		1

Armrest, hemiplegic


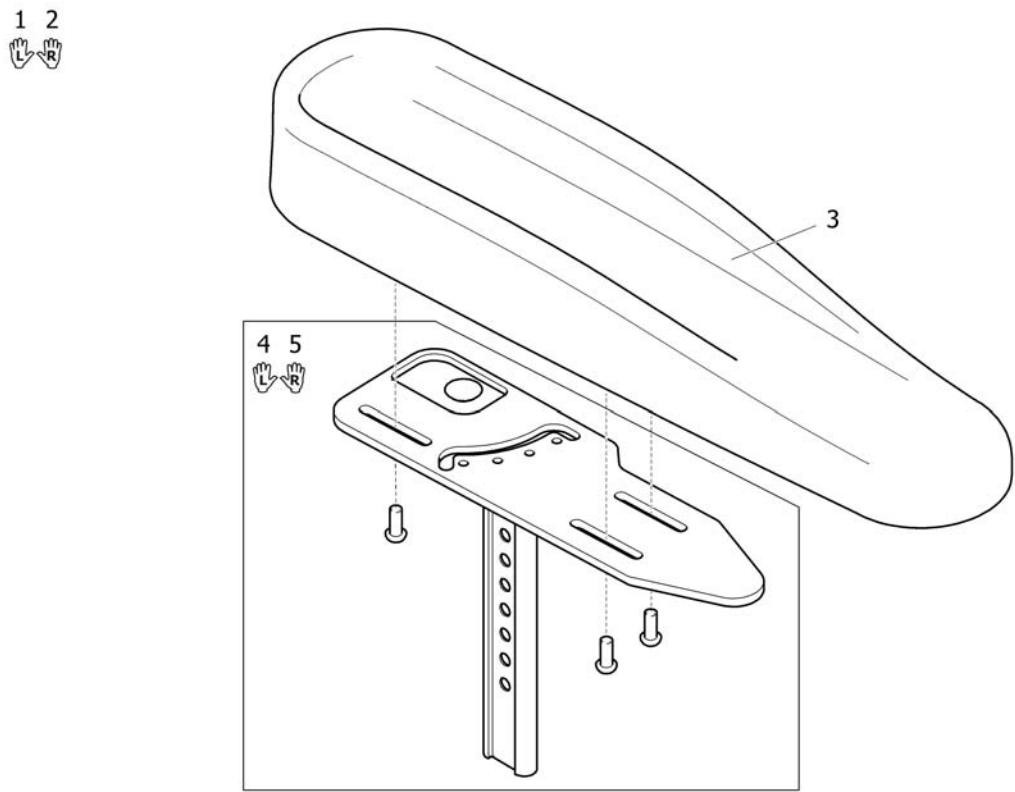


Pos.	Part number	Description	Valid from → until	Qty
1	1493220	Lh hemi dartex armpad complete		1
2	1493219	Rh hemi dartex armpad complete		1
3	1493251	Hemi dartex armpad		1
4	1494006	Support Hemi armrest Lh		1
5	1494005	Support Hemi armrest Rh		1

Pad, siderest

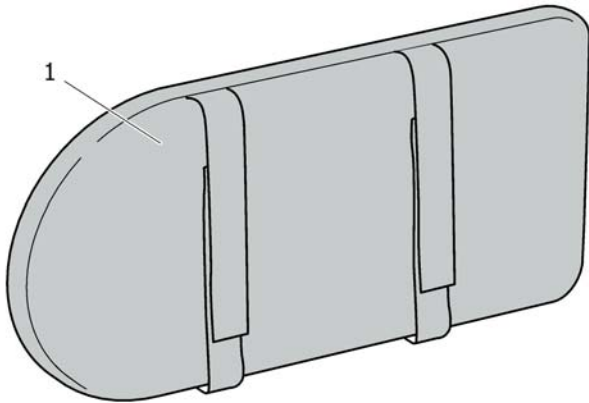


SP1563943

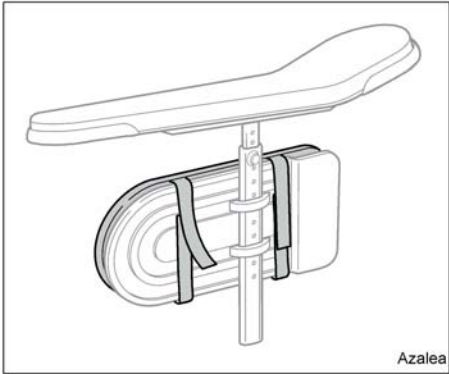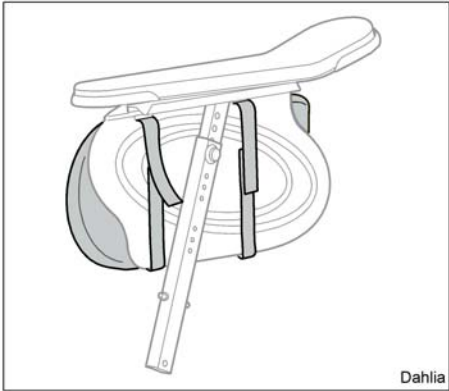

Pos.	Part number	Description	Valid from → until	Qty
1	SP1563943	SIDEREST PAD		1


Armrest attachment

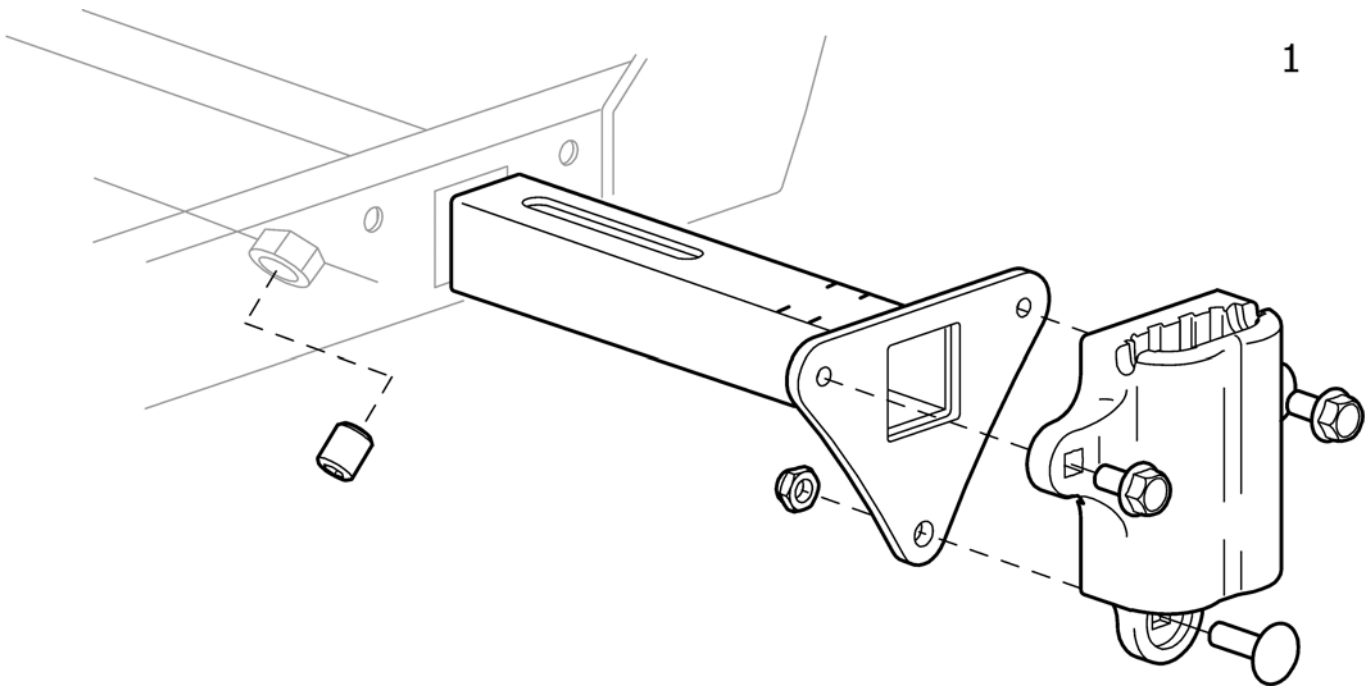

Pos.	Part number	Description	Valid from → until	Qty
1	1451864	ARMREST ATTACHMENT ADJ., AZALEA		1


Armrest attachment, Azalea Max

1

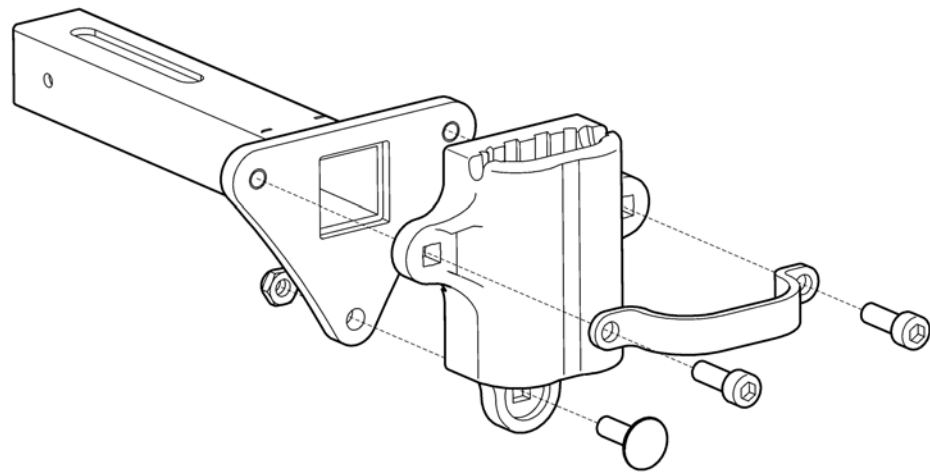

Pos.	Part number	Description	Valid from → until	Qty
1	1525035	Armrest Attachment, Azalea Max		1




Castor wheels

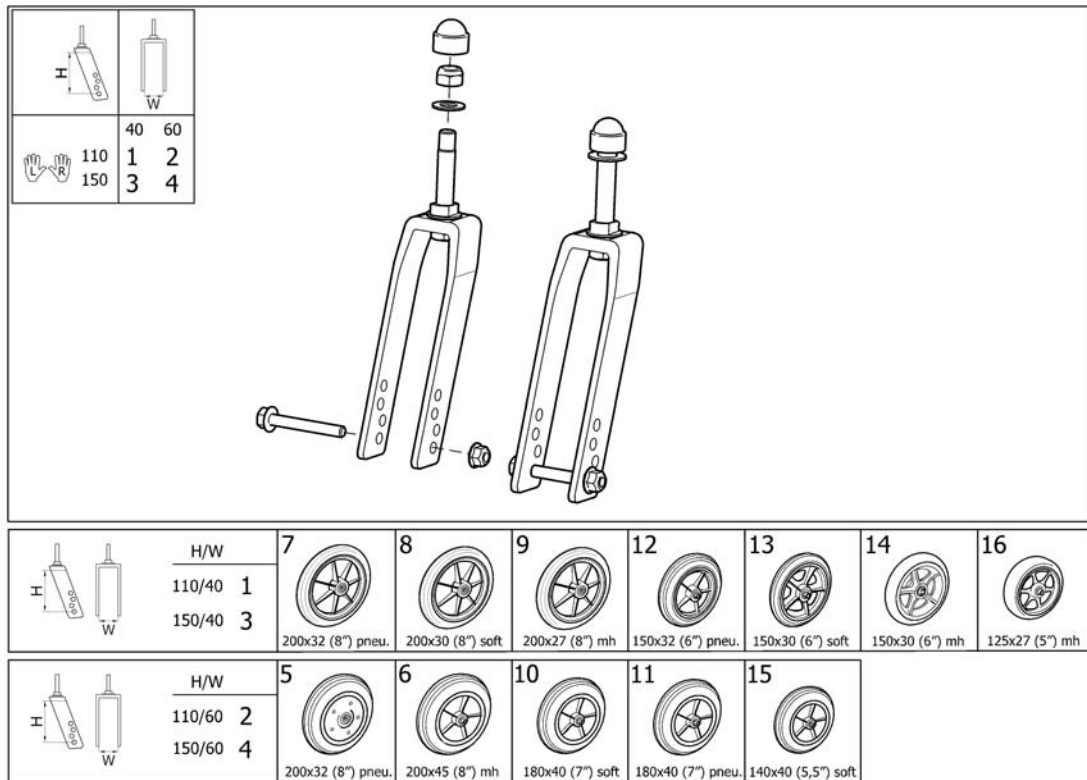


1595320674

Pos.	Part number	Description	Valid from → until	Qty
1	1493442	FORK 110/40, FIXED, CPL, PAIR		1
2	1493443	FORK 110/60, FIXED, CPL, PAIR		1
3	1493440	FORK 150/40, FIXED, CPL, PAIR		1
4	1493441	FORK 150/60, FIXED, CPL, PAIR		1
5	0038N10005	Castor 200x50, Pneumatic		1
6	1418196	Castor With Spoked Rim 200X45 Solid		1
7	5322005	CASTOR 200x32 (8") PNEUMATIC		1
8	5750800	CASTOR 200x30 (8") SOFT		1
9	5322004	200 X 27 Semi Rigid Front Wheel		1
10	5750781	CASTOR 180x40 (7") SOFT		1
11	5750782	CASTOR 180x40 (7") PNEUMATIC		1
12	5322006	CASTOR 150x32 (6") PNEUMATIC		1
13	5750806	CASTOR 150X30 (6") SOFT (S)		1
14	1555797	CASTOR 150x30 (6") MH		1
15	5750734	Wheel Ø140X37 (5.5") soft		1
16	1491228	CASTOR 125X27 (5") MH	2008-10-09 →	1

Wheels 16"

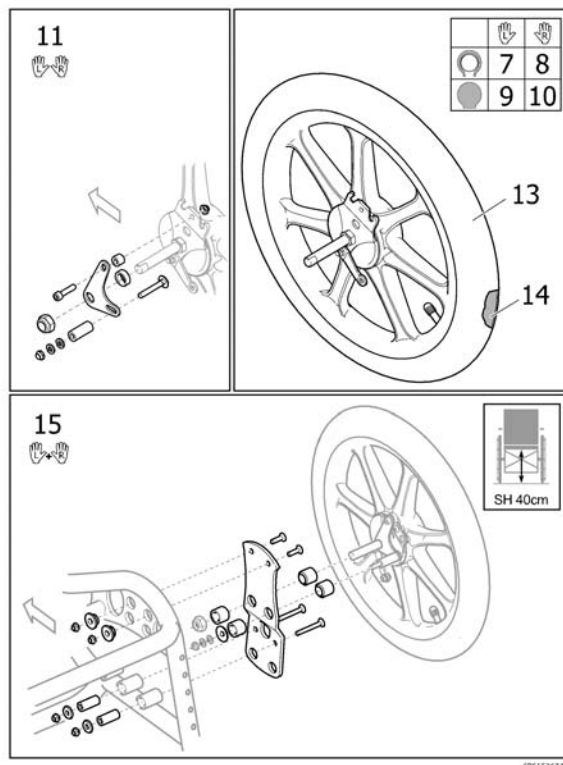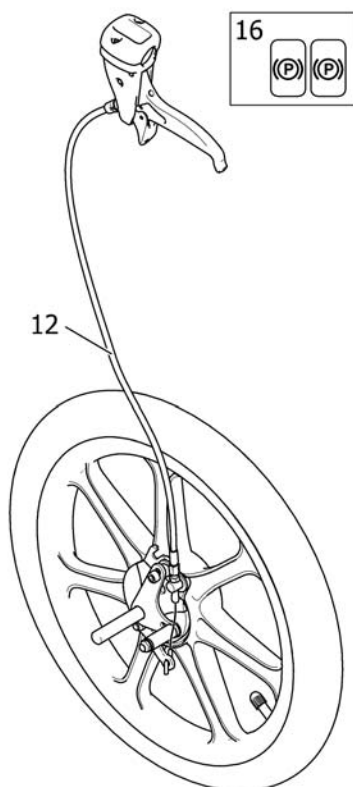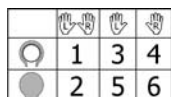


Pos.	Part number	Description	Valid from → until	Qty
1	1536342	WHEEL 16" DRUMBRAKE PNEU PAIR		1
2	1536343	WHEEL 16" DRUMBRAKE SOLID PAIR		1
3	1536344	WHEEL 16" DRUMBRAKE PNEU, LE INCL. CABLE ASSY.		1
4	1536370	WHEEL 16" DRUMBRAKE PNEU, RI INCL. CABLE ASSY.		1
5	1536371	WHEEL 16" DRUMBRAKE SOLID, LE INCL. CABLE ASSY.		1
6	1536372	WHEEL 16" DRUMBRAKE SOLID, RI INCL. CABLE ASSY.		1
7	1537134	WHEEL 16" PNEU DB LEFT		1
8	1537133	WHEEL 16" PNEU DB RIGHT		1
9	1537136	WHEEL 16" SOLID DB LEFT		1
10	1537135	WHEEL 16" SOLID DB RIGHT		1
11	1492298	ATTACHMENT, DRUMBRAKE, 16" WHEEL, AZALEA		1
12	SP1527425	Cable assy 1235mm, brake		1
13	1440029	Tyre 16"x1,75 Pneu		1
14	5750778	Tube 16" x 1.75		1
15	1508785	ATTACHMENT PLATES 16" PAIR		1
16	1536764	DECAL, CABLE ASSEMBLY, BRAKE		1

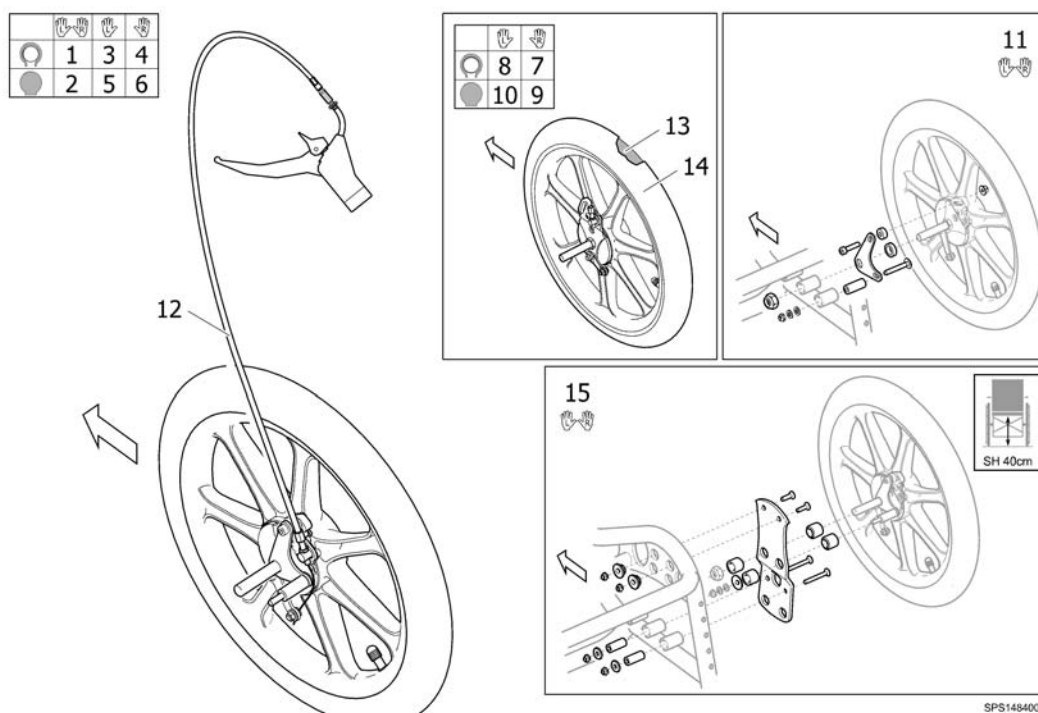Wheels 16" (=>2010.11)

Pos.	Part number	Description	Valid from → until	Qty
1	1492380	WHEEL 16" DRUMBRAKE PNEU., PAIR, AZALEA		1
2	1492381	WHEEL 16" DRUMBRAKE SOLID, PAIR, AZALEA		1
3	1484001	WHEEL 16" DRUMBRAKE PNEU. + CABLE ASSAY, CPL, LE, AZALEA		1
4	1484000	WHEEL 16" DRUMBRAKE PNEU. + CABLE ASSAY, CPL, RI, AZALEA		1
5	1484003	WHEEL 16" DRUMBRAKE SOLID + CABLE ASSAY, CPL, LE, AZALEA		1
6	1484002	WHEEL 16" DRUMBRAKE SOLID + CABLE ASSAY, CPL, RI, AZALEA		1
7	1434825	Wheel 16" Pneu., Drumbrake, Right Mbl		1
8	1434826	Wheel 16" Pneu., Drumbrake, Left Mbl		1
9	1434827	Wheel 16" Solid, Drumbrake, Right Mbl		1
10	1434828	Wheel 16" Solid, Drumbrake, Left Mbl		1
11	1533828	Attachment, 16" wheel, pair, Azalea		1
12	5735266	WIRE CONTROL L=870 CPL		1
13	1440029	Tyre 16"x1,75 Pneu		1
14	5750778	Tube 16" x 1.75		1
15	1508785	ATTACHMENT PLATES 16" PAIR		1

Wheels 24" Basic


24"

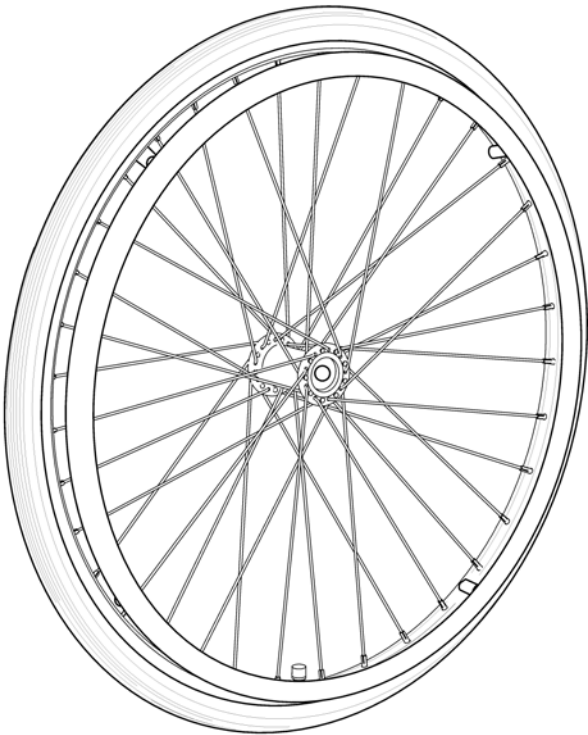


Pos.	Part number	Description	Valid from → until	Qty
1	1498423	24" x 1.3/8" wheel,spoked,anodized handrim,pneumatic		1
2	1498263	24" x 1.3/8" wheel,spoked,anodized handrim,solid tyre,		1

Wheels 24" Basic DB

24" DB

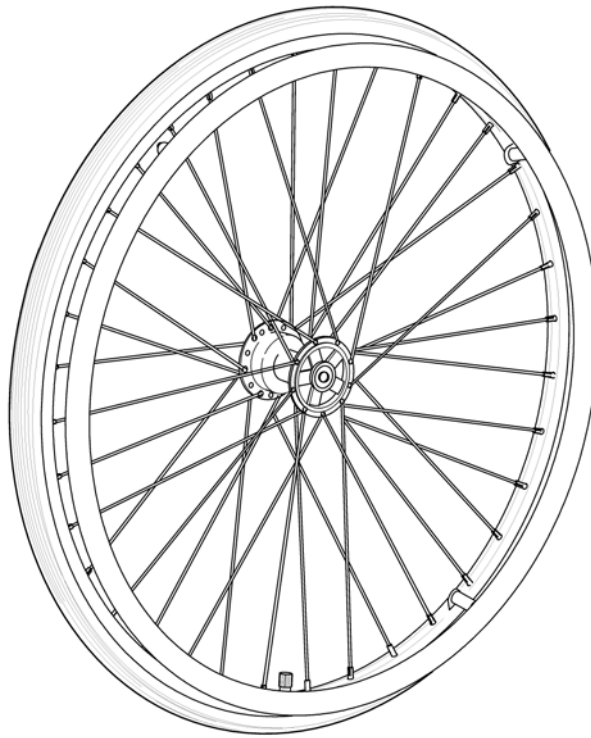

Pos.	Part number	Description	Valid from → until	Qty
1	1492644	Wheel 24" Basic Db, Pneumatic, Profiled + Handrim 21", Alu.		1
2	1492645	Wheel 24" Basic Db, Solid + 21" Alu.		1

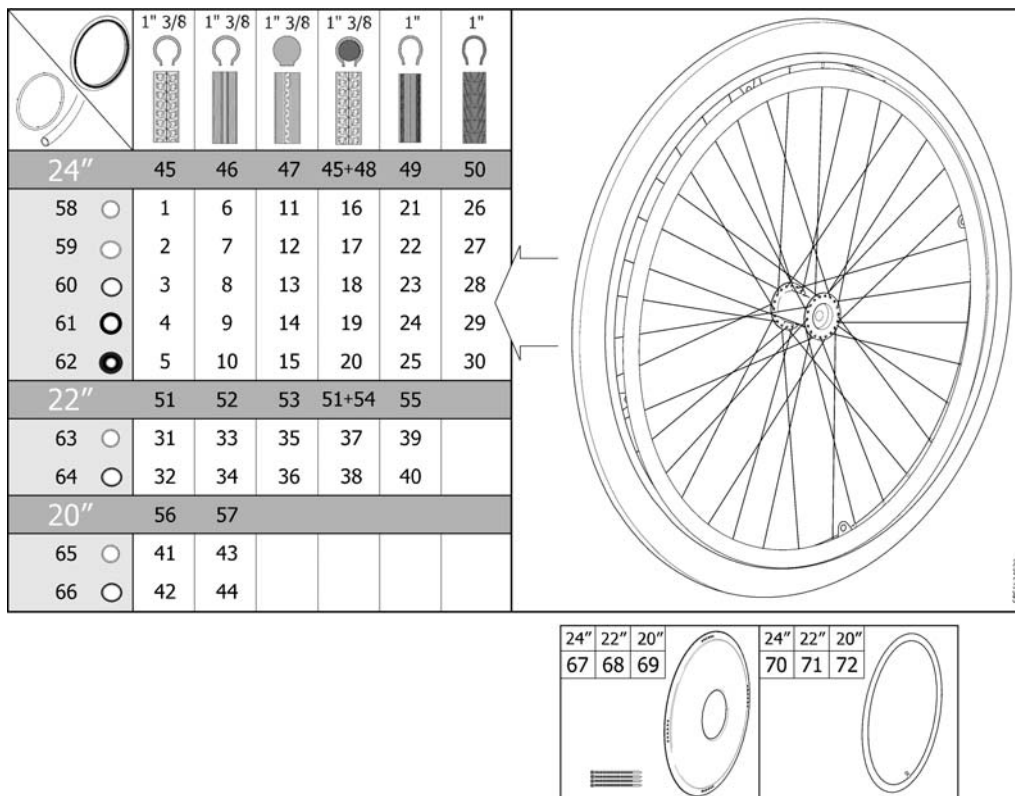Wheel 24-22-20"

Pos.	Part number	Description	Valid from → until	Qty
1	1524030	Wheel 24" Pneu. Profiled + Handrim, Alu, Std2010		1
2	1524186	Wheel 24" Pneu. Profiled + Handrim, Plastic Coated, Std2010		1
3	1530764	Wheel 24" Pneu. Profiled + Handrim, Stainless, Std2010		1
4	1531508	Wheel 24" Pneu. Profiled + Handrim, Foam Coated, Std2010		1
5	1531563	Wheel 24" Pneu. Profiled + Handrim, Blackgrip, Std2010		1
6	1524029	Wheel 24" Pneu. Non-Marking + Handrim, Alu, Std2010		1
7	1524163	Wheel 24" Pneu. Non-Marking + Handrim, Plastic Coated, Std2010		1
8	1530763	Wheel 24" Pneu. Non-Marking + Handrim, Stainless, Std2010		1
9	1531462	Wheel 24" Pneu. Non-Marking + Handrim, Foam Coated, Std2010		1
10	1531538	Wheel 24" Pneu. Non-Marking + Handrim, Blackgrip, Std2010		1
11	1524050	Wheel 24" Solid + Handrim, Alu, Std2010		1
12	1524188	Wheel 24" Solid + Handrim, Plastic Coated, Std2010		1
13	1530762	Wheel 24" Solid + Handrim, Stainless, Std2010		1
14	1531531	Wheel 24" Solid + Handrim, Foam Coated, Std2010		1
15	1531564	Wheel 24" Solid + Handrim, Blackgrip, Std2010		1
16	1524150	Wheel 24" Punctureproof + Handrim, Alu, Std2010		1
17	1524189	Wheel 24" Punctureproof + Handrim, Plastic Coated, Std2010		1
18	1531129	Wheel 24" Punctureproof + Handrim, Stainless, Std2010		1
19	1531532	Wheel 24" Punctureproof + Handrim, Foam Coated, Std2010		1
20	1531565	Wheel 24" Punctureproof + Handrim, Blackgrip, Std2010		1
21	1524049	Wheel 24x1" Low Profile + Handrim, Alu., Std2010		1
22	1524187	Wheel 24x1" Low Profile + Handrim, Plastic Coated, Std2010		1
23	1531063	Wheel 24x1" Low Profile + Handrim, Stainless Steel, Std2010		1
24	1531533	Wheel 24x1" Low Profile + Handrim, Foam Coated, Std2010		1
25	1531566	Wheel 24x1" Low Profile + Handrim, Black Grip, Std2010		1
26	1532143	Wheel 24" Black Marathon + Handrim, Alu., Std2010		1
27	1532183	Wheel 24" Black Marathon + Handrim, Plastic Coated, Std2010		1
28	1532144	Wheel 24" Black Marathon + Handrim, Stainless Steel, Std2010		1
29	1532184	Wheel 24" Black Marathon + Handrim, Foam Coated, Std2010		1
30	1532185	Wheel 24" Black Marathon + Handrim, Black Grip, Std2010		1
31	1523949	Wheel 22" Pneu. Profiled + Handrim, Alu, Std2010		1

Wheel 24-22-20"

Pos.	Part number	Description	Valid from → until	Qty
32	1524024	Wheel 22" Pneu. Profiled + Handrim, Plastic Coated, Std2010		1
33	1523948	Wheel 22" Pneu. Non-Marking + Handrim, Alu, Std2010		1
34	1524023	Wheel 22" Pneu. Non-Marking + Handrim, Plastic Coated, Std2010		1
35	1524021	Wheel 22" Solid + Handrim, Alu, Std2010		1
36	1524027	Wheel 22" Solid + Handrim, Plastic Coated, Std2010		1
37	1524022	Wheel 22" Punctureproof + Handrim, Alu, Std2010		1
38	1524028	Wheel 22" Punctureproof + Handrim, Plastic Coated, Std2010		1
39	1524020	Wheel 22x1" Low Profile + Handrim, Alu., Std2010		1
40	1524026	Wheel 22x1" Low Profile + Handrim, Plastic Coated, Std2010		1
41	1523944	Wheel 20" Pneu. Profiled + Handrim, Alu, Std2010		1
42	1523945	Wheel 20" Pneu. Profiled + Handrim, Plastic Coated, Std2010		1
43	1523885	Wheel 20" Pneu. Non-Marking + Handrim, Alu, Std2010		1
44	1523902	Wheel 20" Pneu. Non-Marking + Handrim, Plastic Coated, Std2010		1
45	5750019	Tyre 24"x1 3/8", Profiled		1
46	5750014	Tyre 24"x1 3/8" (32-540), Non-Marking		1
47	5750015	Tyre 24"x1 3/8", Solid		1
48	5207046	Tube 24"x1 3/8", Flex		1
49	5750160	Tyre 24"x1" (25-540), Non-Marking		1
50	1512742	Tyre 24"x1" (25-540), Black Marathon		1
51	1531455	Tyre 22"x1 3/8" (37-501) Profiled		1
52	1531456	Pneumatic tyre, 22" x 1", HP, non marking		1
53	5750172	Tyre 22" Solid		1
54	5750732	Tube 22"x1 3/8", Flex		1
55	1531454	Tyre 22"x1" (25-501) Low Profile		1
56	5750017	Tyre 20"x1 3/8", Profiled		1
57	5750012	Tyre 20"x1 3/8", Non-Marking		1
58	1531577	Handrim Aluminium Cpl, 24" Wheel, Std2010		1
59	1531578	Handrim Plastic Cpl, 24" Wheel, Std2010		1
60	1531305	Handrim Stainless Cpl, 24" Wheel, Std2010		1
61	1531459	Handrim Foam Coated Cpl, 24" Wheel, Std2010		1
62	1531460	Handrim Black Grip Cpl, 24" Wheel, Std2010		1
63	1531579	Handrim Aluminium Cpl, 22" Wheel, Std2010		1
64	1531580	Handrim Plastic Cpl, 22" Wheel, Std2010		1
65	1530271	Handrim Aluminium Cpl, 20" Wheel, Std2010		1
66	1530272	Handrim Plastic Cpl, 20" Wheel, Std2010		1
67	5102807	Spoke Protection 24" Frosted Cpl		1
68	5102800	Spoke Protection 22" Frosted Cpl		1
69	5102795	Spoke Protection 20" Frosted Cpl		1
70	5750036	Tube 24"x1 3/8"		1
70	5750161	Tube 24"x1" (25-540)		1
71	5750035	Tube 22"x1 3/8"		1
72	5750034	Tube 20"x1 3/8"		1


Handrim Stainless Cpl, 24" Wheel, Std2010

Pos.	Part number	Description	Valid from → until	Qty
3854	1528924	SKRUV MRX M5X40 FZB, KLASS 4.8 PHIL, DIN 7985		4
3854	1529757	DRIVRING 24" ROSTFRI		1
3854	1531457	DISTANS DRIVRING L=22MM		4
3854	1531458	DISTANS DRIVRING L=5MM		4

Wheel 24-22" DB

	1" 3/8	1" 3/8	1" 3/8	1" 3/8	1"
24"	23	24	25	23 + 26	27
33 ○	1	4	7	10	
34 ○	2	5	8	11	
35 ○	3	6	9	12	13
22"	28	29	30	28 + 31	32
36 ○	14	16	18	20	
37 ○	15	17	19	21	
38 ○					22

24" 22"	24" 22"
39 40	41 42

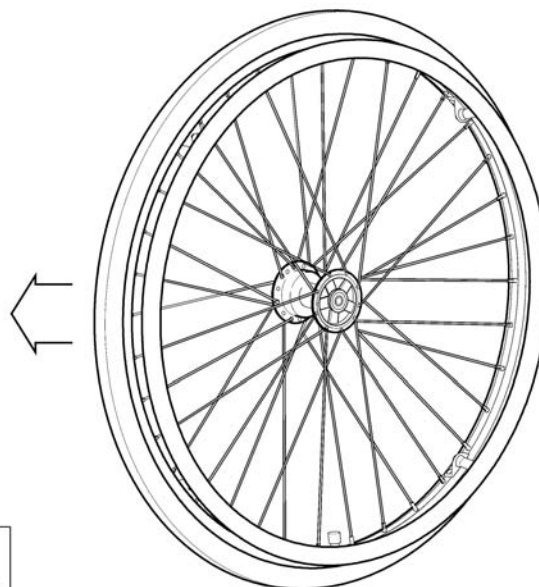


5051314416

Pos.	Part number	Description	Valid from → until	Qty
1	1524437	Wheel 24" Db Pneu. Profiled + Handrim, Alu, Std2010		1
2	1524514	Wheel 24" Db Pneu. Profiled + Handrim, Plastic Coated, Std2010		1
3	1531080	Wheel 24" Db Pneu. Profiled + Handrim, Stainless, Std2010		1
4	1524436	Wheel 24" Db Pneu. Non-Marking + Handrim, Alu, Std2010		1
5	1524513	Wheel 24" Db Pneu. Non-Marking + Handrim, Plastic Coated, Std2010		1
6	1531064	Wheel 24" Db Pneu. Non-Marking + Handrim, Stainless, Std2010		1
7	1524447	Wheel 24" Db Solid + Handrim, Alu, Std2010		1
8	1524515	Wheel 24" Db Solid + Handrim, Plastic Coated, Std2010		1
9	1531127	Wheel 24" Db Solid + Handrim, Stainless, Std2010		1
10	1524512	Wheel 24" Db Punctureproof + Handrim, Alu, Std2010		1
11	1524516	Wheel 24" Db Punctureproof + Handrim, Plastic Coated, Std2010		1
12	1531128	Wheel 24" Db Punctureproof + Handrim, Stainless, Std2010		1
13	1536613	Wheel 24x1" Db Low Profile + Handrim, Stainless, Std2010		1
14	1524290	Wheel 22" Db Pneu. Profiled + Handrim, Alu, Std2010		1
15	1524361	Wheel 22" Db Pneu. Profiled + Handrim, Plastic Coated, Std2010		1
16	1524289	Wheel 22" Db Pneu. Non-Marking + Handrim, Alu, Std2010		1
17	1524337	Wheel 22" Db Pneu. Non-Marking + Handrim, Plastic Coated, Std2010		1
18	1524291	Wheel 22" Db Solid + Handrim, Alu, Std2010		1
19	1524388	Wheel 22" Db Solid + Handrim, Plastic Coated, Std2010		1
20	1524292	Wheel 22" Db Punctureproof + Handrim, Alu, Std2010		1
21	1524435	Wheel 22" Db Punctureproof + Handrim, Plastic Coated, Std2010		1
22	1536612	Wheel 22x1" Db Low Profile + Handrim, Stainless, Std2010		1
23	5750019	Tyre 24"x1 3/8", Profiled		1
24	5750014	Tyre 24"x1 3/8" (32-540), Non-Marking		1
25	5750015	Tyre 24"x1 3/8", Solid		1
26	5207046	Tube 24"x1 3/8", Flex		1
27	5750160	Tyre 24"x1" (25-540), Non-Marking		1
28	1531455	Tyre 22"x1 3/8" (37-501) Profiled		1
29	1531456	Pneumatic tyre, 22" x 1", HP, non marking		1
30	5750172	Tyre 22" Solid		1
31	5750732	Tube 22"x1 3/8", Flex		1

Wheel 24-22" DB

Pos.	Part number	Description	Valid from → until	Qty
32	1531454	Tyre 22"x1" (25-501) Low Profile		1
33	1531577	Handrim Aluminium Cpl, 24" Wheel, Std2010		1
34	1531578	Handrim Plastic Cpl, 24" Wheel, Std2010		1
35	1531305	Handrim Stainless Cpl, 24" Wheel, Std2010		1
36	1531579	Handrim Aluminium Cpl, 22" Wheel, Std2010		1
37	1531580	Handrim Plastic Cpl, 22" Wheel, Std2010		1
38	1536546	Handrim, Stainless Cpl, 22" Wheel, Std2010		1
39	5102807	Spoke Protection 24" Frosted Cpl		1
40	5102800	Spoke Protection 22" Frosted Cpl		1
41	5750036	Tube 24"x1 3/8"		1
42	5750035	Tube 22"x1 3/8"		1


1 2

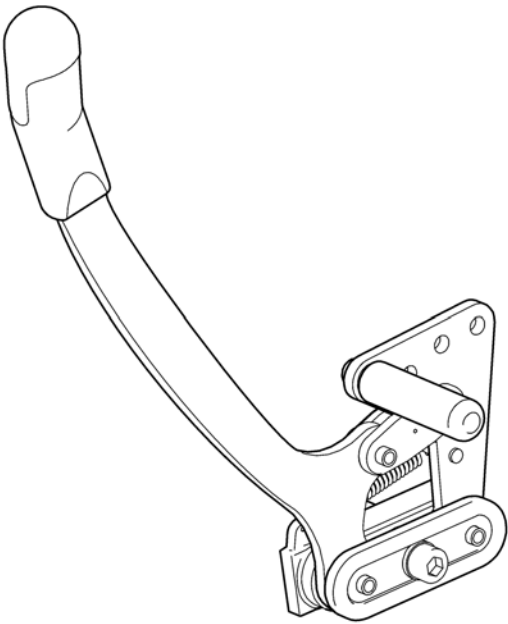

SPS1486410


Pos.	Part number	Description	Valid from → until	Qty
1	1486411	BRAKE LEFT, AZALEA		1
2	1486410	BRAKE RIGHT, AZALEA		1

Brake, Azalea Minor/Azalea Base sw34

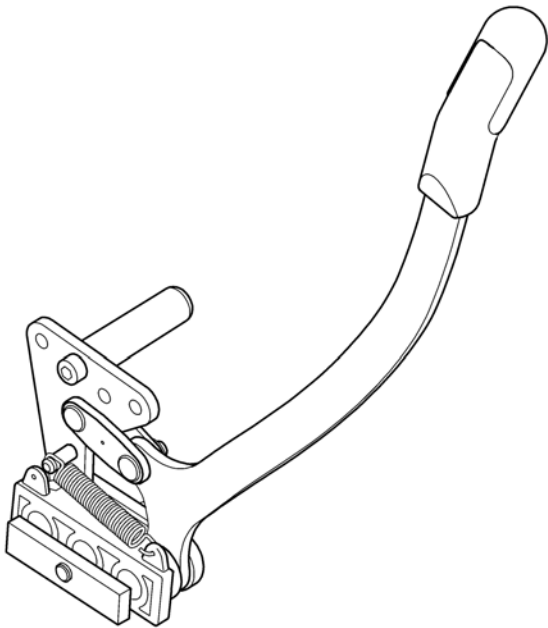

00689815451

Pos.	Part number	Description	Valid from → until	Qty
1	1508981	BRAKE CPL BENT LEVER, LEFT		1
2	1508980	BRAKE CPL BENT LEVER, RIGHT		1

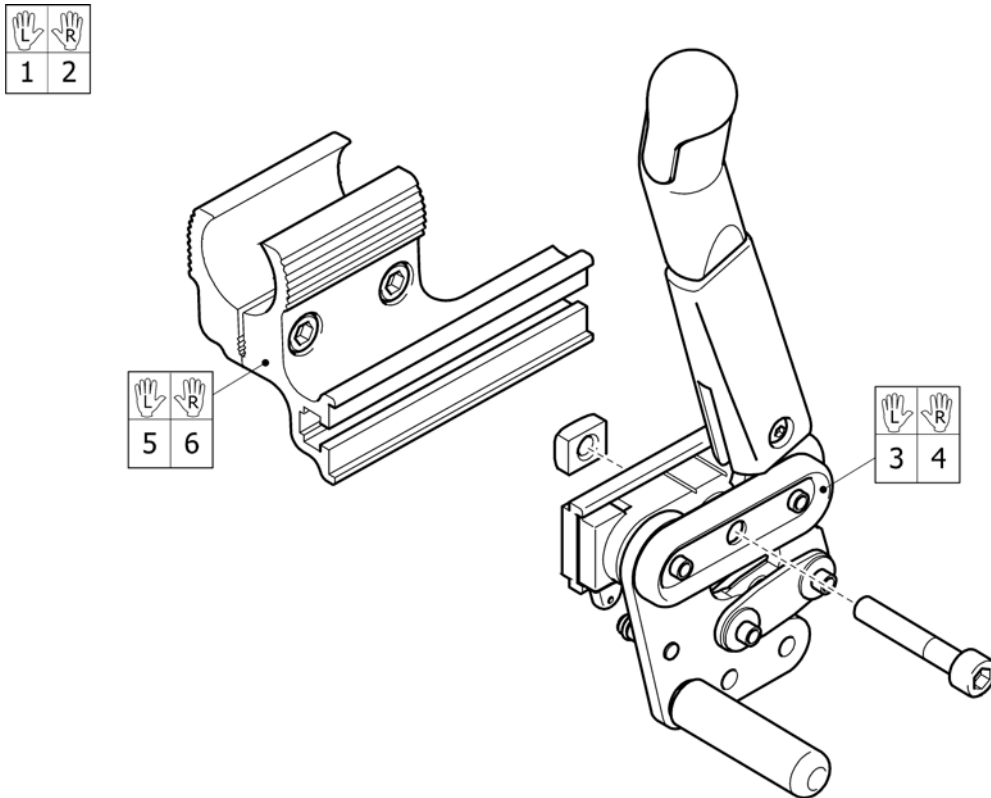
Brake, Azalea Max

Pos.	Part number	Description	Valid from → until	Qty
1	1524719	Brake Extended Le Cpl. With Attachment		1
2	1524720	Brake Extended Ri Cpl. With Attachment		1
3	1535752	Brake Extended, Le.		1
4	1535753	Brake Extended, Ri.		1
5	1518621	Attachment Le, Brake Extended		1
6	1518620	Attachment Ri, Brake Extended		1

Attachment Ri, Brake Extended

Pos.	Part number	Description	Valid from → until	Qty
DG	5221304	BRAKE CLAMP INNER 25		1
DG	5226143	BRAKE CLAMP OUTER RI,XLT		1
DG	5724089	SCREW MC6S M6X25 FZB 8.8 DIN 912		2



Brake, extended, Azalea Max

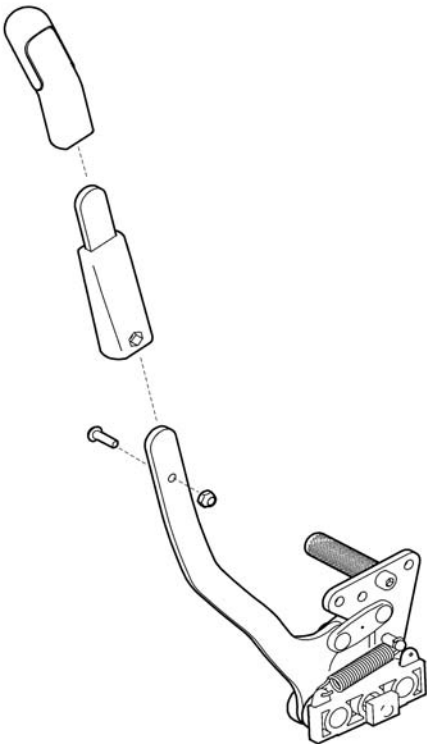


SP1563919

Pos.	Part number	Description	Valid from → until	Qty
1	SP1563920	Brake, extended, left cpl. (incl attachment)		1
2	SP1563919	Brake, extended, right cpl. (incl attachment)		1
3	SP1570441	Brake, Extended, Left		1
4	SP1570440	Brake, Extended, Right		1
5	1518621	Attachment Le, Brake Extended		1
6	1518620	Attachment Ri, Brake Extended		1

Brake, Azalea Tall

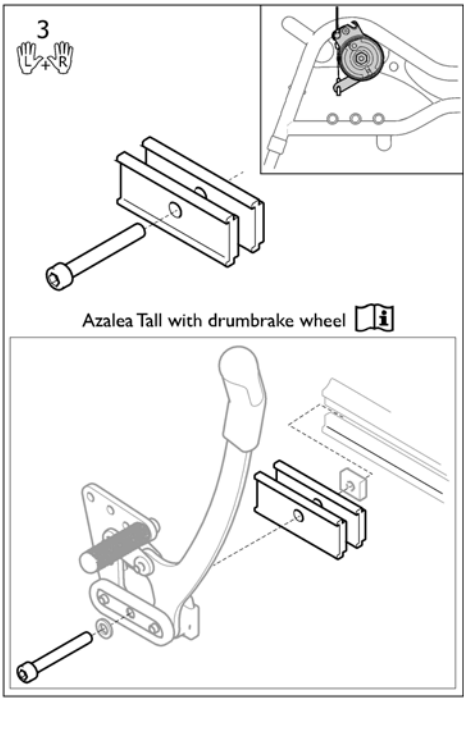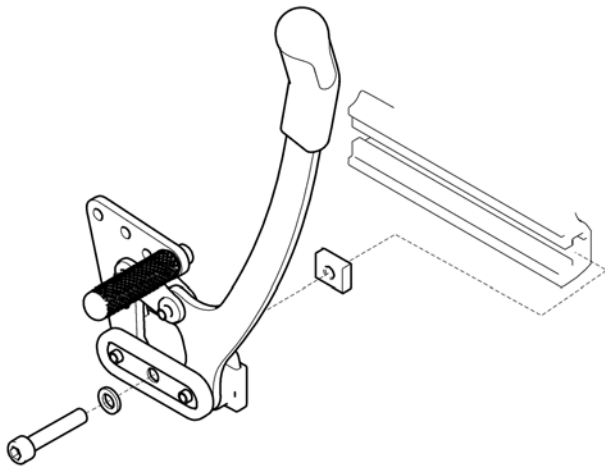


Pos.	Part number	Description	Valid from → until	Qty
1	1486411	BRAKE LEFT, AZALEA		1
2	1486410	BRAKE RIGHT, AZALEA		1
3	SP1517109	SPACER KIT 10MM , BRAKE		1

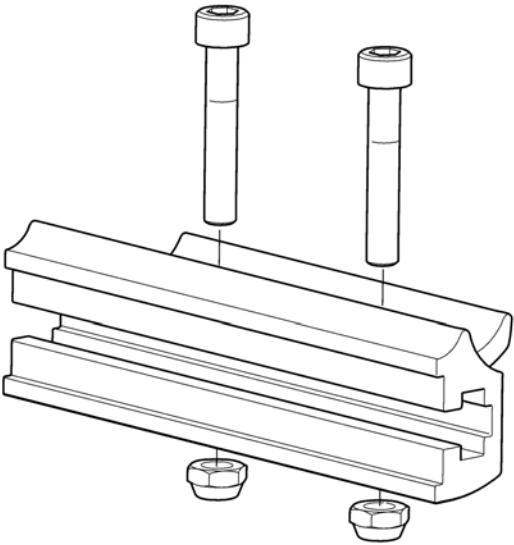

SPS149901




Pos.	Part number	Description	Valid from → until	Qty
	1491317	BRAKE ATTACHMENT		1

Drumbrake, assistant manoeuvred

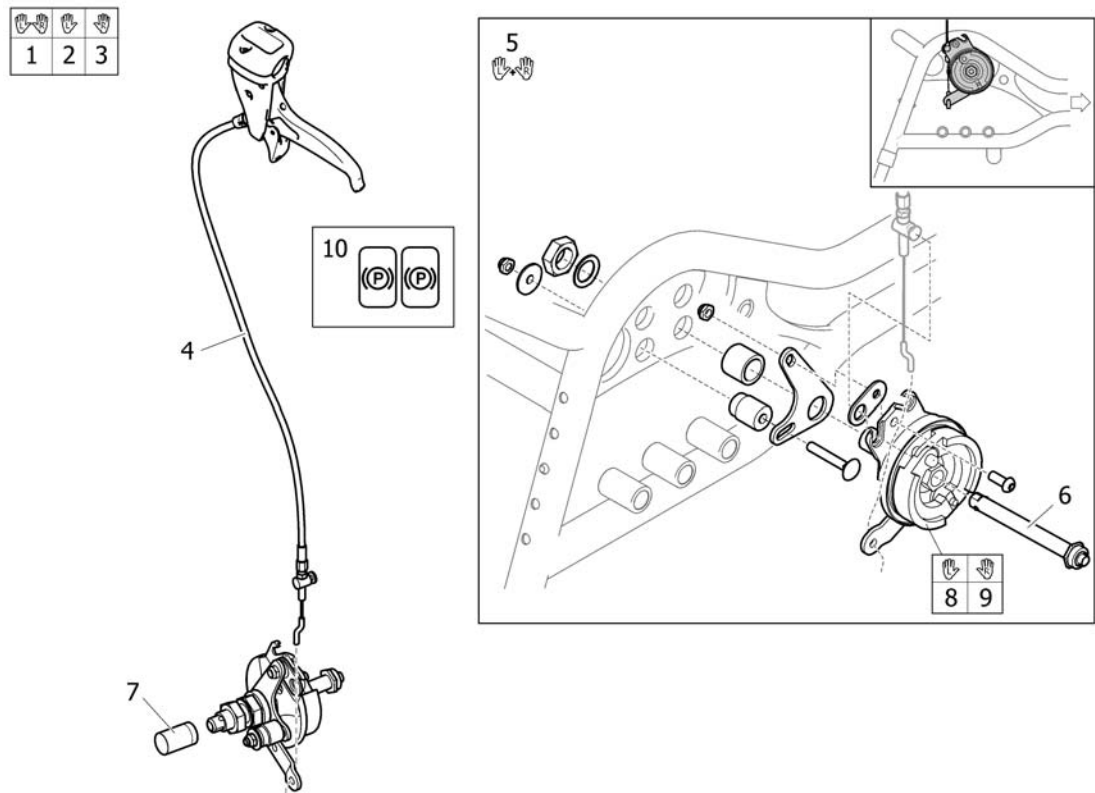




Pos.	Part number	Description	Valid from → until	Qty
1	1536757	BRAKE KIT DB CPL, PAIR		1
2	1536758	BRAKE KIT DB LEFT, CPL		1
3	1536759	BRAKE KIT DB RIGHT, CPL		1
4	SP1527425	Cable assy 1235mm, brake		1
5	1536543	BRAKE KIT DB CPL, PAIR		1
6	5750192	Axle, Drumbrake		1
7	5100925	PROTECTION CAP		1
8	1536252	DRUMBRAKE UNIT, RIGHT, MBL		1
9	1536253	DRUMBRAKE UNIT, LEFT, MBL		1
10	1536764	DECAL, CABLE ASSEMBLY, BRAKE		1

Drumbrake, assistant manoeuvred (=>11.2010)

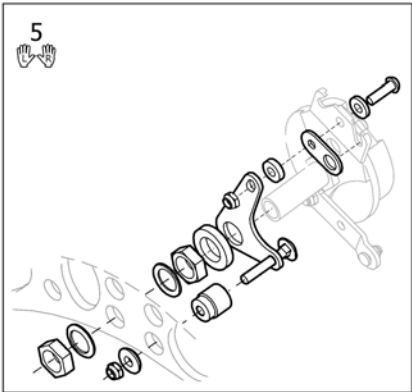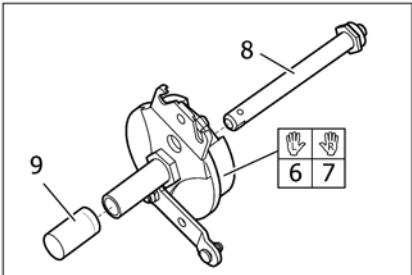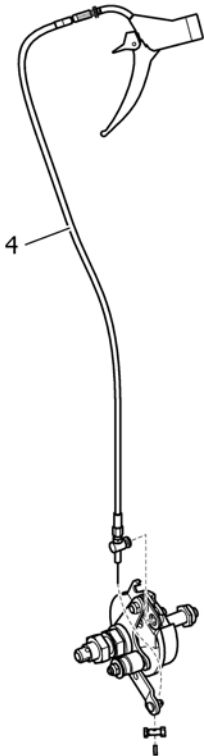

Pos.	Part number	Description	Valid from → until	Qty
1	1512948	BRAKE KIT 24" DB CPL, AZALEA		1
2	1448799	Brake kit 24" DB left, cpl. Azalea		1
3	1448798	Brake kit 24" DB right, cpl. Azalea		1
4	5735266	WIRE CONTROL L=870 CPL		1
5	1492232	REAR WHEEL ATTACHMENT, PAIR, CLEMATIS		1
6	5750199	Drumbrake Unit, Lh		1
7	5750705	Drumbrake Unit, Rh		1
8	5750192	Axle, Drumbrake		1
9	5100925	PROTECTION CAP		1


Brake, user manoeuvred, 16" wheel

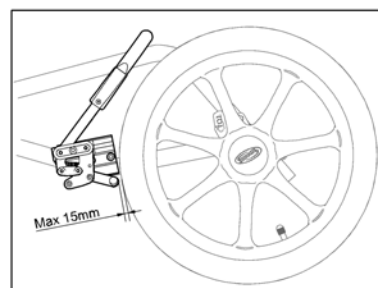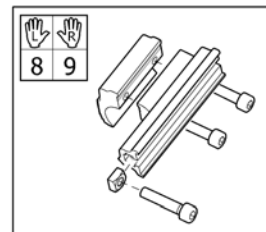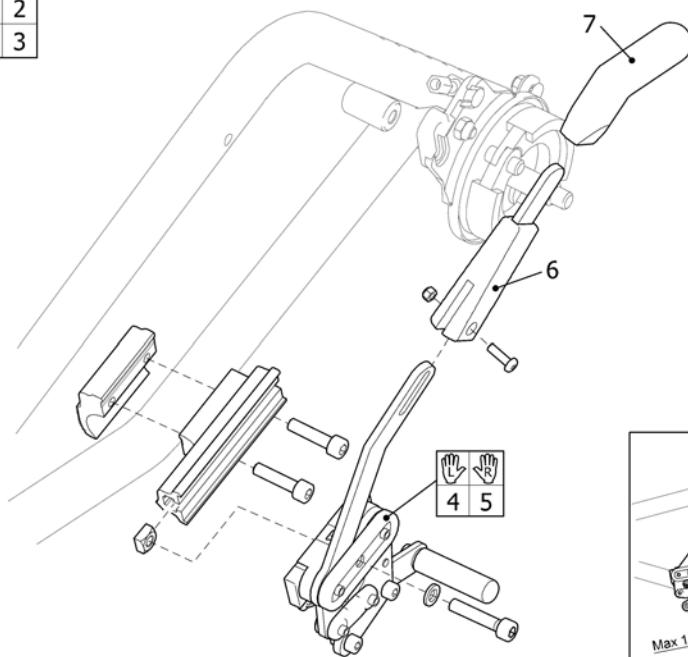

Pos.	Part number	Description	Valid from → until	Qty
1	1444335	BRAKE USER MANOEUVURED PAIR CPL 16"WHEEL ASSIST		1
2	1446119	BRAKE USER MANOEUVURED LE CPL 16" WHEEL ASSIST		1
3	1446118	BRAKE USER MANOEUVURED RI CPL 16" WHEEL ASSIST		1
4	1446117	BRAKE BODY LE, USER BRAKE, 16" WHEEL		1
5	1446116	BRAKE BODY RI, USER BRAKE, 16" WHEEL		1
6	SP5203340	Brake arm extension, cpl.		1
7	SP5221280	Handle grip for parking brake		1
8	5240240	ATTACHMENT, BRAKE, LH, CPL		1
9	5240241	ATTACHMENT, BRAKE, RH, CPL		1

Pelvic belt


1

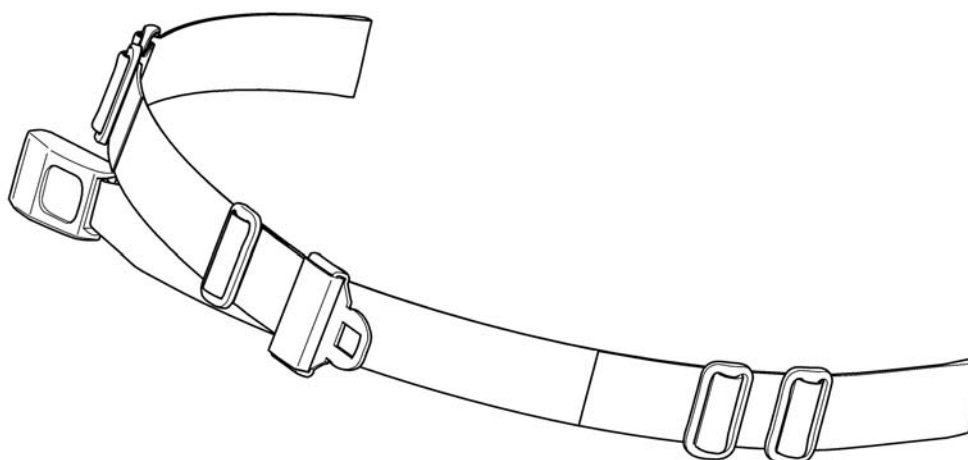

1551066

Pos.	Part number	Description	Valid from → until	Qty
1	1551066	Pelvic belt		1

1

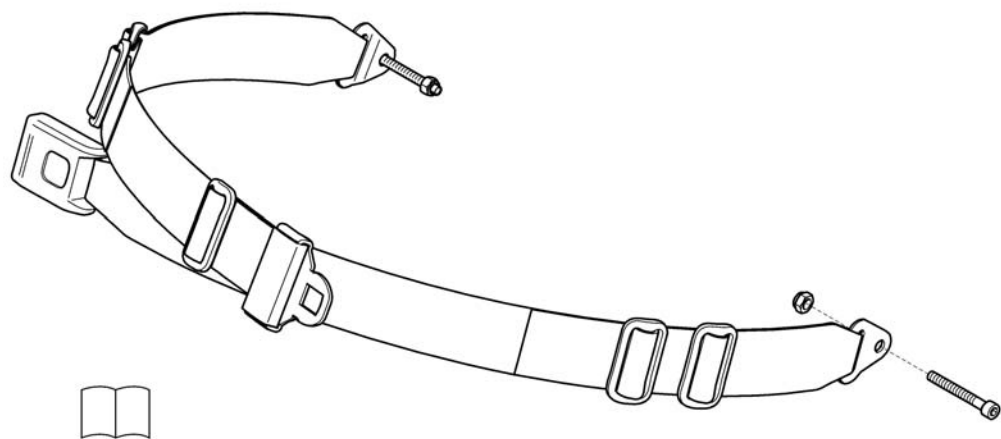


SPS1542016

Pos.	Part number	Description	Valid from → until	Qty
1	1543660	PELVIC BELT INCL. ATTACHMENTS		1

Pelvic belt, Max


1

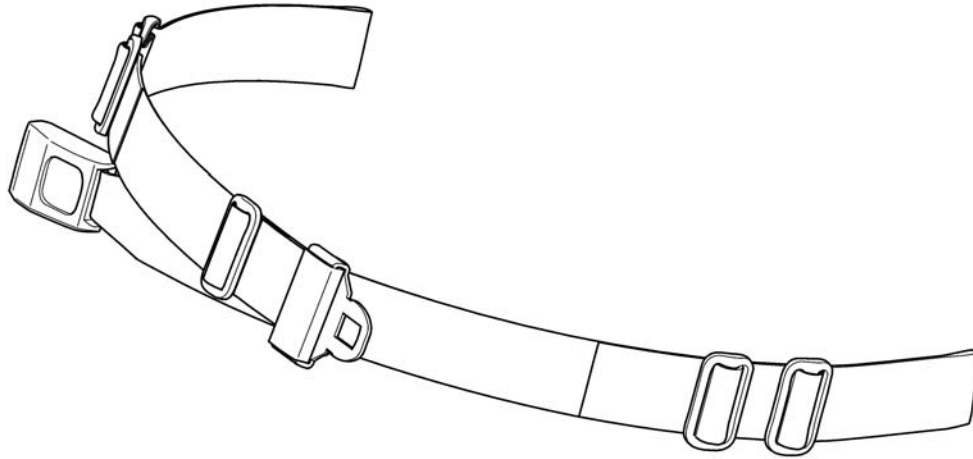

1551067

Pos.	Part number	Description	Valid from → until	Qty
1	1551067	Lap Belt Autolock, long		1


1

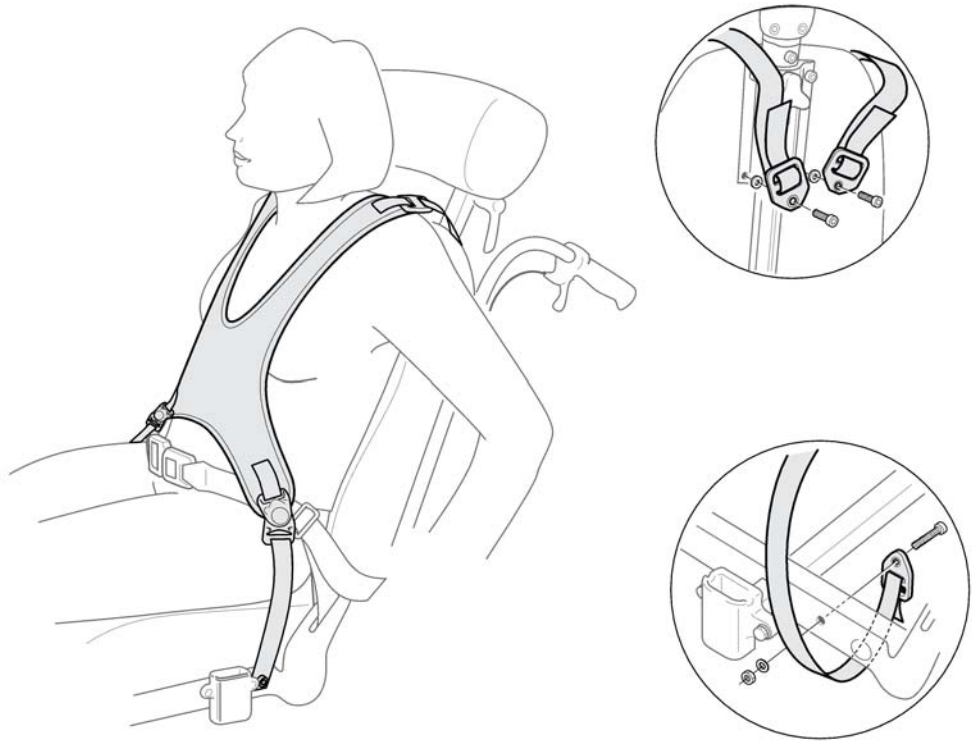

Pos.	Part number	Description	Valid from → until	Qty
1	1512914	X-harness incl. fixation		1

Anti-tip device

1

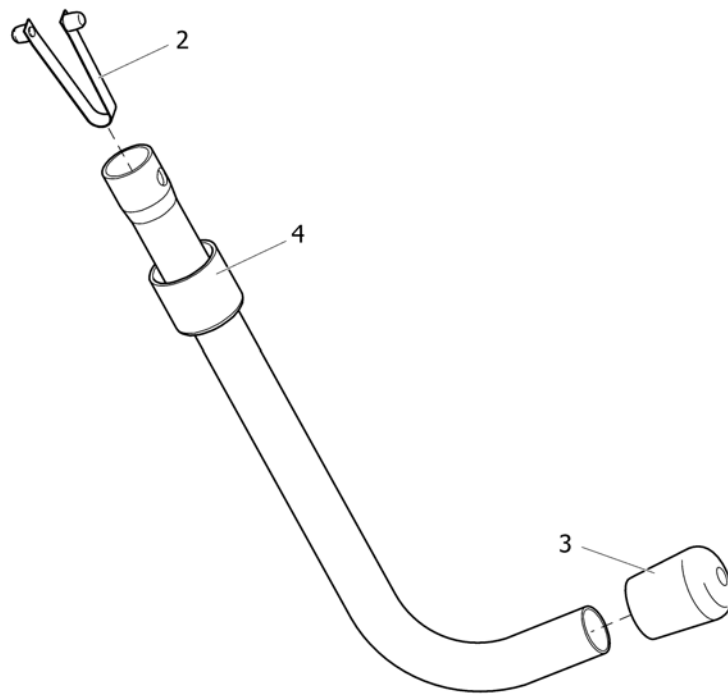


SP5103109


Pos.	Part number	Description	Valid from → until	Qty
1	1439933	ANTI-TIP DEVICE CPL		1
2	5321593	SPRING LOCK (DOBBLE)		1
3	5716493	END CAP Ø18, RUBBER BLACK		1
4	SP5103109	ENDSLEEVE FOOTREST 10-PACK		1

Reflector

		
	20-26"	16"
<i>Azalea</i>	1	2
<i>Bellis</i>	3	3
<i>Clematis</i>	1	-
<i>Focus</i>	4	8
<i>Spin^x</i>	1	-
<i>Spirea², Spirea³, Spirea^{4NG}</i>	4	5
<i>Wheeler</i>	6	7
<i>XLT, XLT Max</i>	3	-

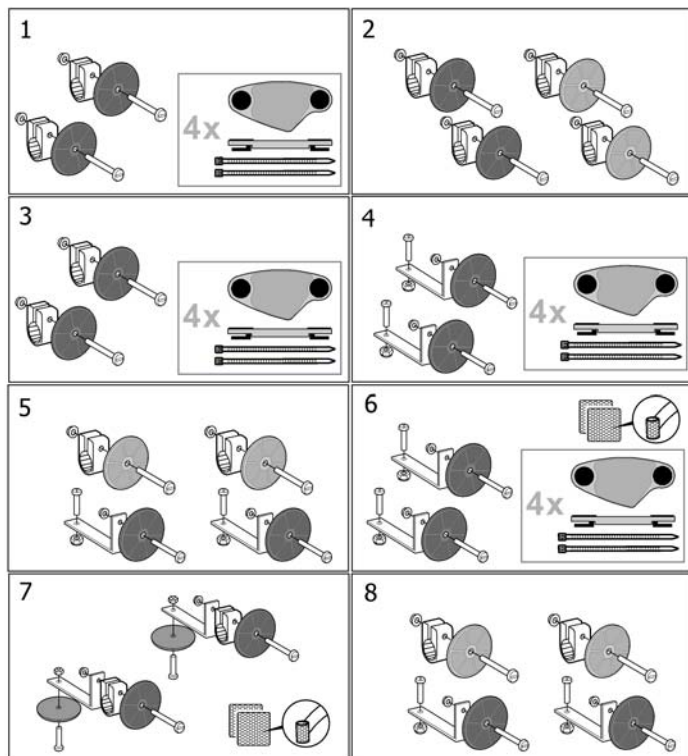


Pos.	Part number	Description	Valid from → until	Qty
1	SP5206655	Reflector kit 20-24"		1
2	SP5254249	Reflector kit 16"		1
3	5206640	Reflector Kit 3		1
4	5104965	Reflector Kit 5		1
5	1435760	Reflector Kit 6		1
6	5108147	Reflector Kit 7		1
7	5108024	Reflector Kit 8		1
8	1546559	Reflector Kit 9		1

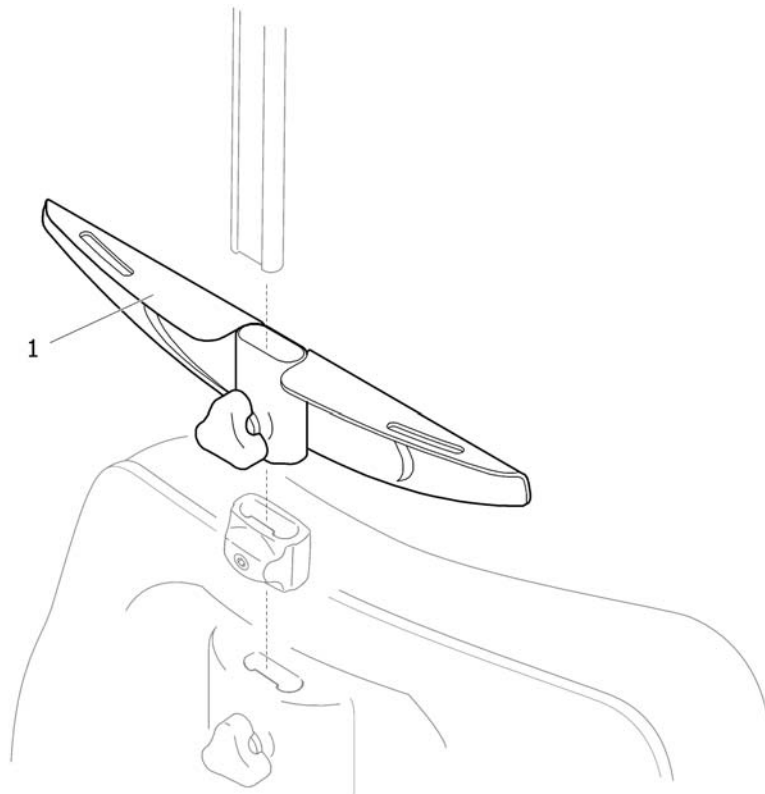Attachment, X-harness, Flex3


Pos.	Part number	Description	Valid from → until	Qty
1	SP1559963	Attachment, X-harness, Flex3		1

Headrest

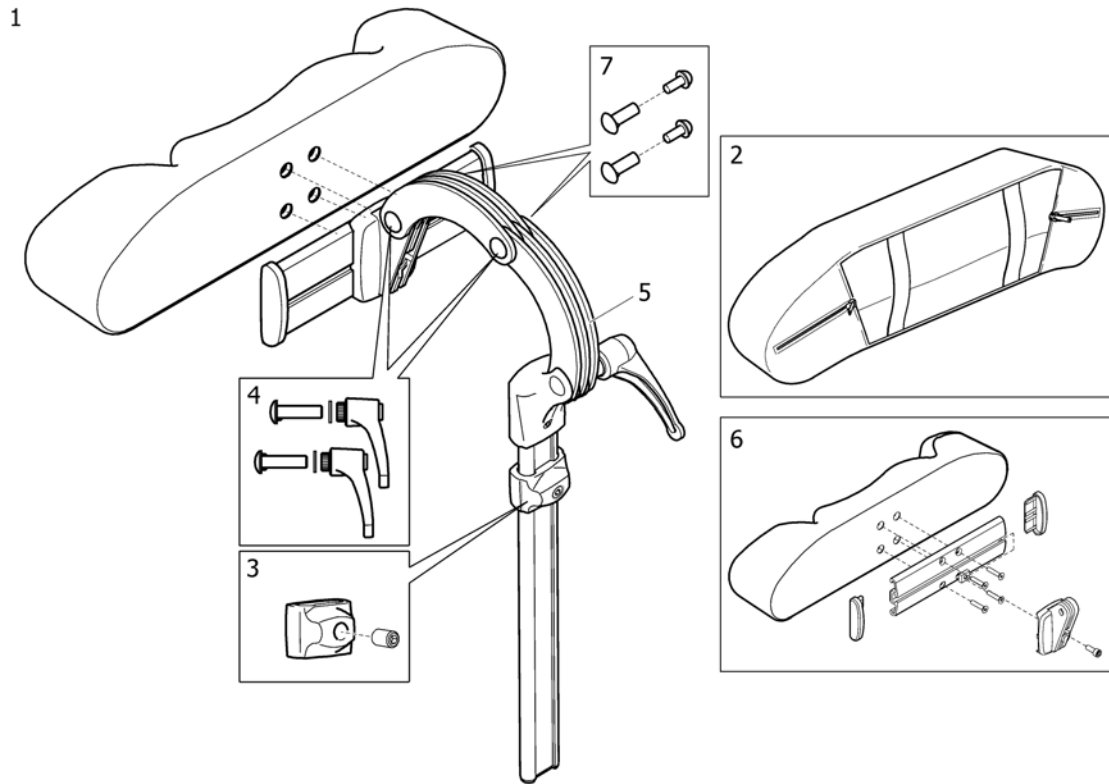

SP1144559

Pos.	Part number	Description	Valid from → until	Qty
1	1446559	HEADREST TR26 CPL SIDEWISE ADJUSTABLE		1
2	1518516	Cover for headrest TR26		1
3	1448661	stop neck and headrest		1
4	1445693	HANDLES, CLINIC KIT NECK- AND HEADREST		1
5	1485538	Headrest fixation kit		1
6	1530755	HEADREST CUSHION CPL		1
7	SP1572769	FASTENERS KIT, HEAD- & NECKREST		1

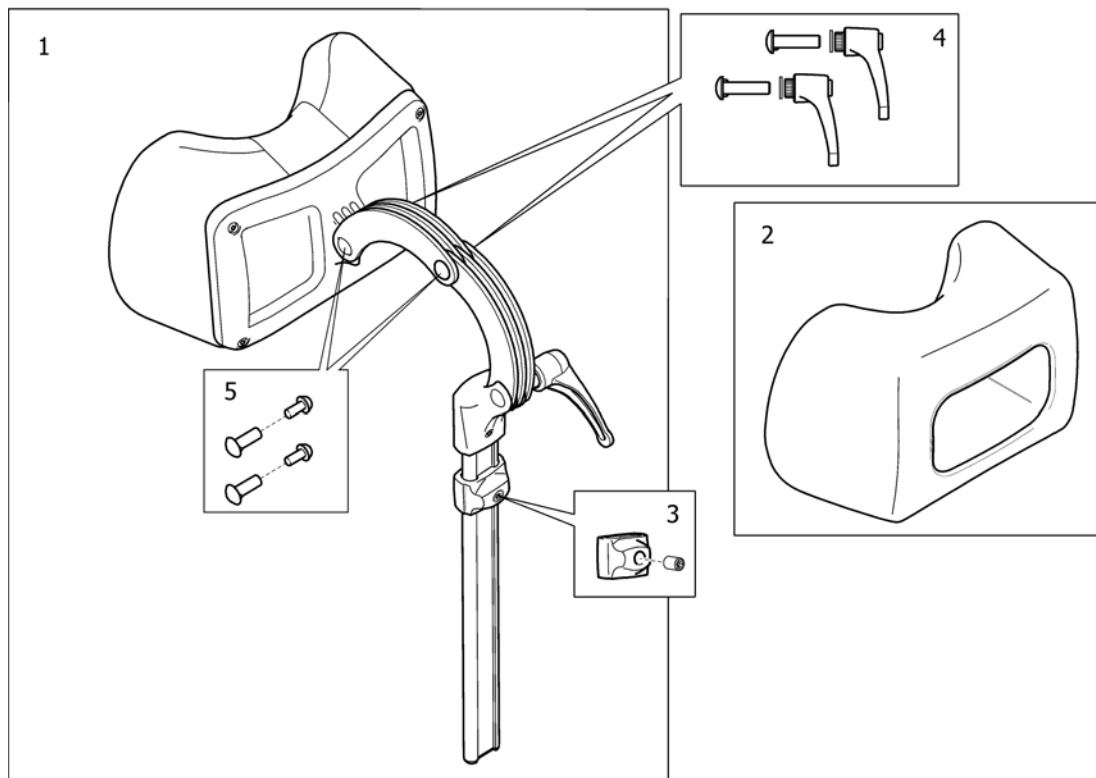
Neckrest

SP150848

Pos.	Part number	Description	Valid from → until	Qty
1	1508848	NECKREST, CPL		1
2	1518517	Cover neckrest TR34		1
3	1448661	stop neck and headrest		1
4	1445693	HANDLES, CLINIC KIT NECK- AND HEADREST		1
5	SP1572769	FASTENERS KIT, HEAD- & NECKREST		1

Cheek support

1

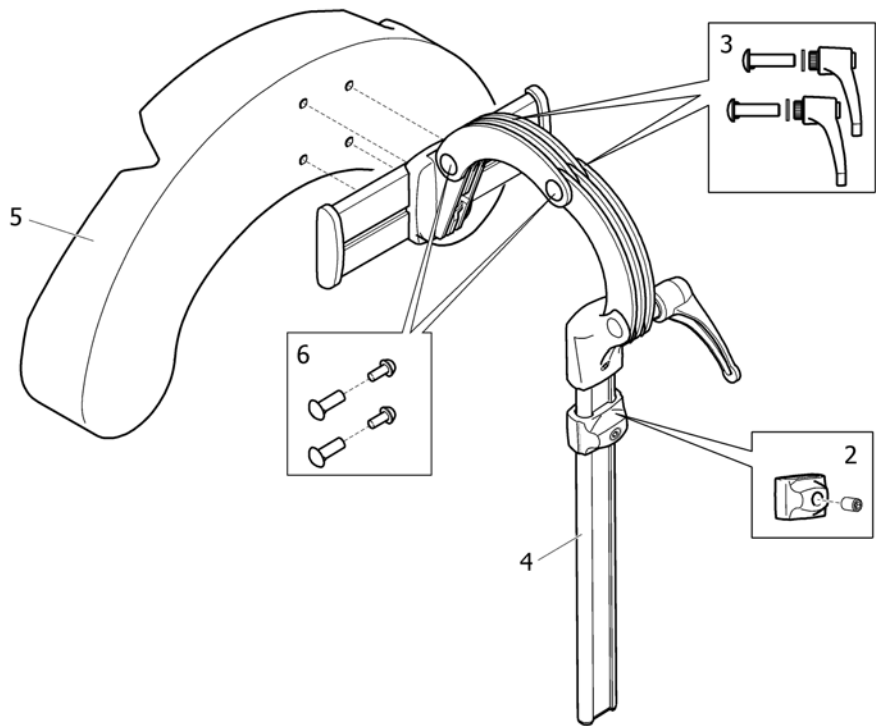


SP11446569

Pos.	Part number	Description	Valid from → until	Qty
1	1446560	HEAD/CHEEK SUPP. TR26 CPL. SIDEWISE ADJUSTABLE		1
2	1448661	stop neck and headrest		1
3	1445693	HANDLES, CLINIC KIT NECK- AND HEADREST		1
4	1485538	Headrest fixation kit		1
5	1444304	Cushion, cheek support		1
6	SP1572769	FASTENERS KIT, HEAD- & NECKREST		1


Pommel

1

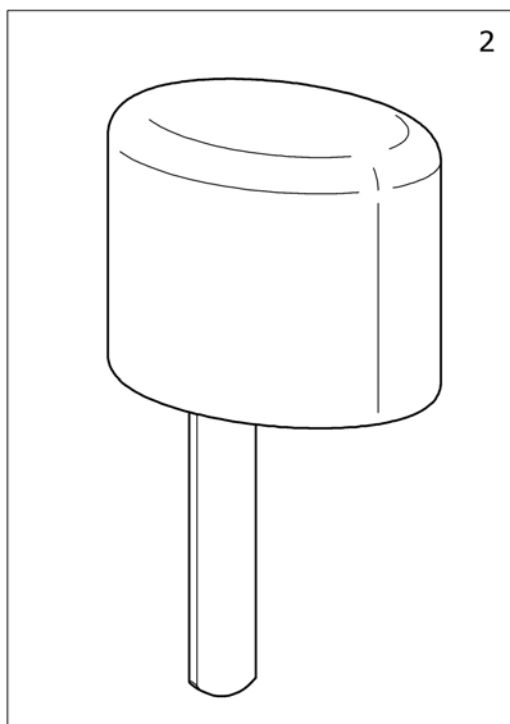


3

Rea *Azalea*
Rea *Dahlia*

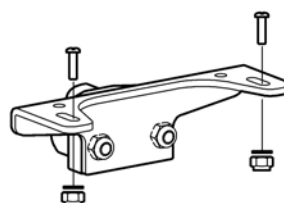

3


Rea *Clematis*

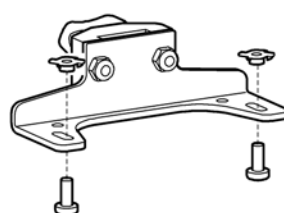

SP1444172


Pos.	Part number	Description	Valid from → until	Qty
1	SP1449897	Pommel, Azalea + Clematis		1
2	1444172	POMMEL, DARTEX BLACK		1
3	SP1450464	Holder pommel, Azalea/Clematis		1

Trunk supports, Flex3 backrest

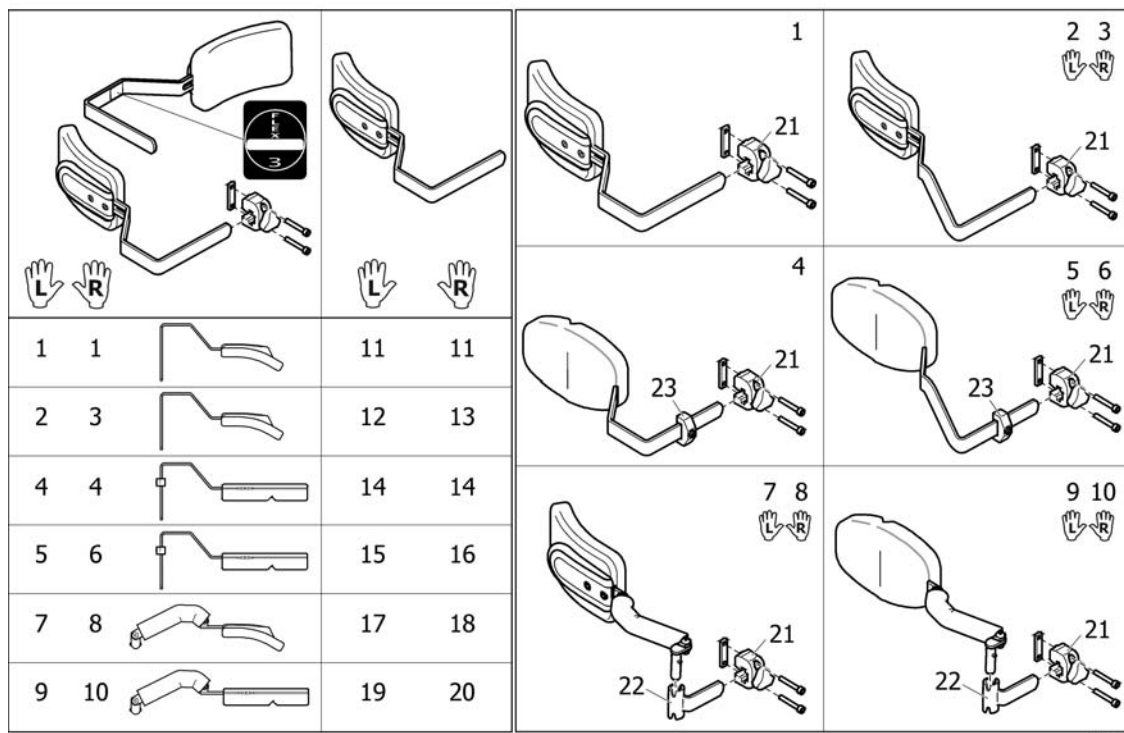

Pos.	Part number	Description	Valid from → until	Qty
1	SP1557672	Trunk support 2012, cpl.		1
2	SP1552680	Trunk support 2012, le, extended, cpl.		1
3	SP1552681	Trunk support 2012, ri, extended, cpl.		1
4	SP1552726	Trunk support 2012, multi adj, cpl.		1
5	SP1552728	Trunk support 2012, le, multi adj, extended, cpl.		1
6	SP1552729	Trunk support 2012, ri, multi adj, extended, cpl.		1
7	SP1557673	Trunk support 2012, le, swingaway, cpl.		1
8	SP1557674	Trunk support 2012, ri, swingaway, cpl.		1
9	SP1557675	Trunk support 2012, le, multi adj, Swingaway, cpl.		1
10	SP1557676	Trunk support 2012, ri, multi adj, Swingaway, cpl.		1
11	SP1552698	Trunk support 2012		1
12	SP1552678	Trunk support 2012, le, extended		1
13	SP1552679	Trunk support 2012, ri, extended		1
14	SP1555690	Trunk support 2012, multi adj.		1
15	SP1555692	Trunk support 2012, le, multi adj, extended		1
16	SP1555691	Trunk support 2012, ri, multi adj, extended		1
17	SP1551853	Trunk Support 2012, Le, Swingaway		1
18	SP1551866	Trunk Support 2012, Ri, Swingaway		1
19	SP1529225	Trunk support, multi adj., left		1
20	SP1529224	Trunk support, multi adj., right		1
21	SP1543574	Attachment, trunk support 2012		1
22	SP1524903	Retainer, Trunk Support		1
23	SP1448728	Memory, Trunk support		1


Trunk supports, Flex3 tension adj. backrest

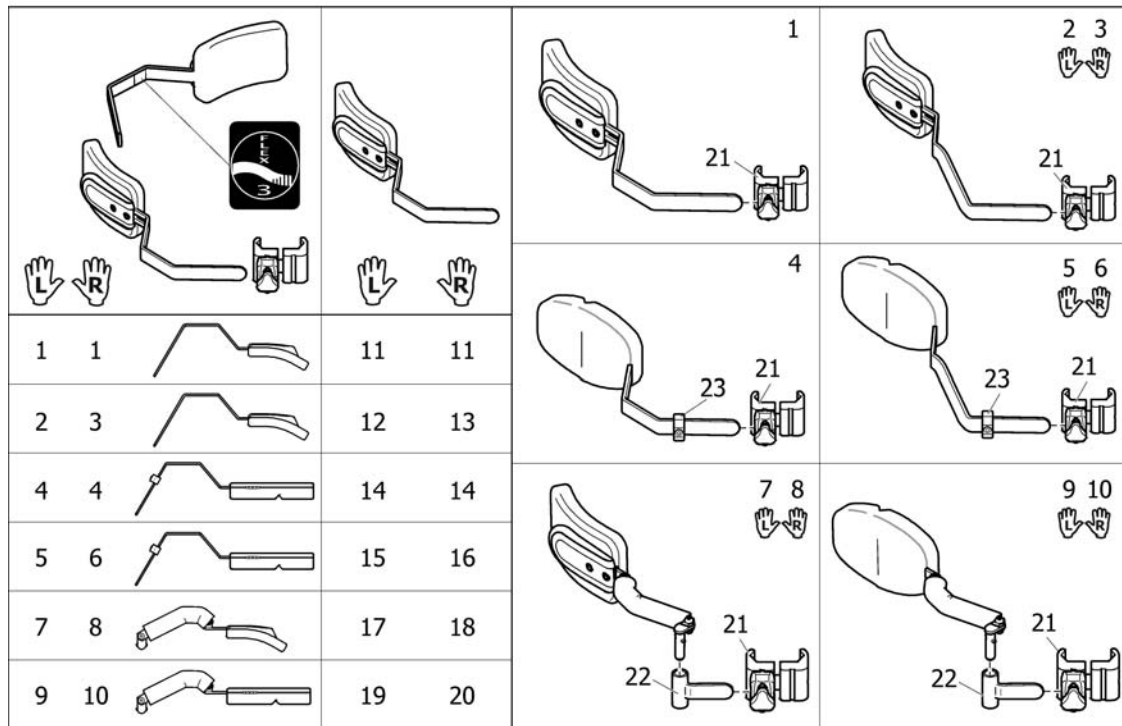

SP156.0288


Pos.	Part number	Description	Valid from → until	Qty
1	SP1561730	Trunk support 2012, cpl, Flex3 tension adj.		1
2	SP1562962	Trunk support 2012, le, extended, cpl, Flex3 tension adj.		1
3	SP1562963	Trunk support 2012, ri, extended, cpl, Flex3 tension adj.		1
4	SP1561729	Trunk support 2012, multi adj, cpl, Flex3 tension adj.		1
5	SP1562960	Trunk support 2012, le, multi adj, extended, cpl, Flex3 tension adj.		1
6	SP1562961	Trunk support 2012, ri, multi adj, extended, cpl, Flex3 tension adj.		1
7	SP1561641	Trunk support 2012, le, swingaway, cpl, Flex3 tension adj.		1
8	SP1561642	Trunk support 2012, ri, swingaway, cpl, Flex3 tension adj.		1
9	SP1561651	Trunk support 2012, le, multi adj, Swingaway, cpl, Flex3 tension adj.		1
10	SP1561652	Trunk support 2012, ri, multi adj, Swingaway, cpl, Flex3 tension adj.		1
11	SP1561728	Trunk support 2012, Flex3 tension adj.		1
12	SP1562922	Trunk support 2012, le, extended, Flex3 tension adj.		1
13	SP1562921	Trunk support 2012, ri, extended, Flex3 tension adj.		1
14	SP1561727	Trunk support 2012, multi adj, Flex3 tension adj.		1
15	SP1562920	Trunk support 2012, le, multi adj, extended, Flex3 tension adj.		1
16	SP1562919	Trunk support 2012, ri, multi adj, extended, Flex3 tension adj.		1
17	SP1551853	Trunk Support 2012, Le, Swingaway		1
18	SP1551866	Trunk Support 2012, Ri, Swingaway		1
19	SP1529225	Trunk support, multi adj., left		1
20	SP1529224	Trunk support, multi adj., right		1
21	SP1554143	Attachment, Trunk Support		1
22	SP1524903	Retainer, Trunk Support		1
23	SP1448728	Memory, Trunk support		1

Trunk supports, Flex2

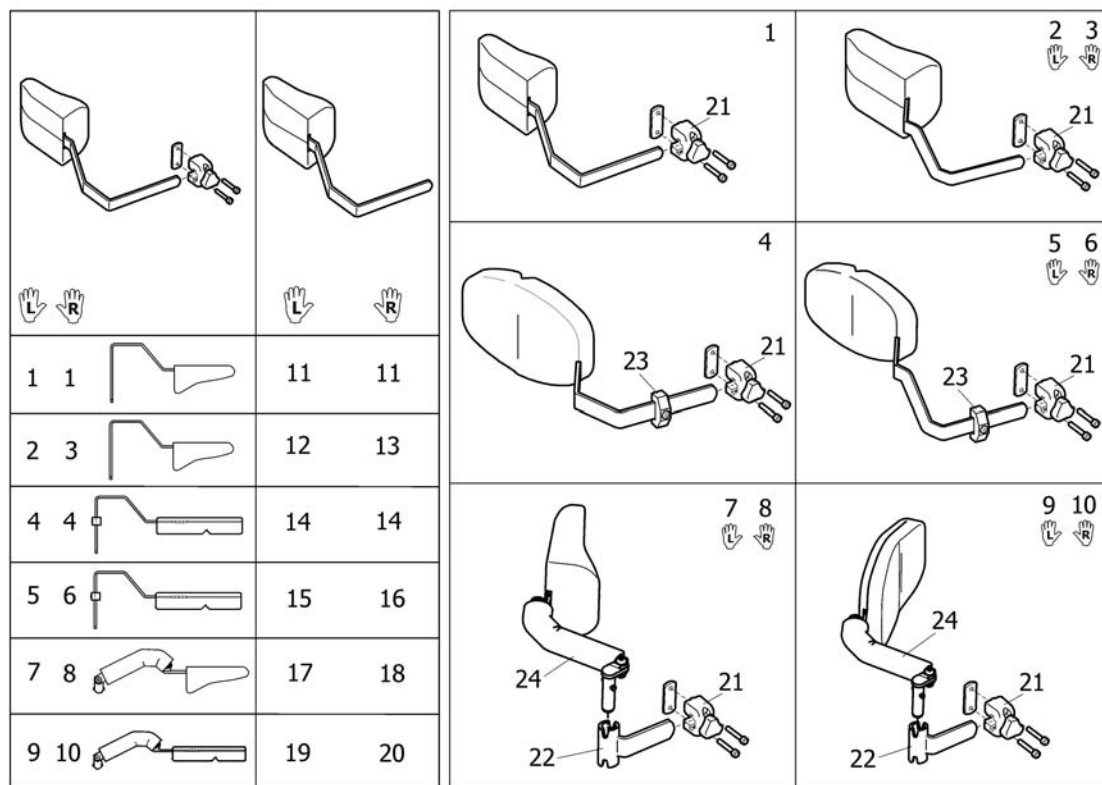


Pos.	Part number	Description	Valid from → until	Qty
1	SP1445798	TRUNK SUPPORT, FIXED, CPL.		1
2	SP1492608	TRUNK SUPPORT, LE, EXTENDED, CPL. (ATTACHMENT INCL.)		1
3	SP1492609	TRUNK SUPPORT, RI, EXTENDED, CPL. (ATTACHMENT INCL.)		1
4	SP1448692	TRUNK SUPPORT, MULTI ADJ. CPL. (ATTACHMENT INCL.)		1
5	SP1484447	TRUNK SUPPORT, LE, MULTI ADJ. EXTENDED, CPL. (ATTACHMENT INCL.)		1
6	SP1484448	TRUNK SUPPORT, RI, MULTI ADJ. EXTENDED, CPL. (ATTACHMENT INCL.)		1
7	SP1448697	TRUNK SUPPORT, LE, SWINGAWAY, CPL. (ATTACHMENT INCL.)		1
8	SP1448698	TRUNK SUPPORT, RI, SWINGAWAY, CPL. (ATTACHMENT INCL.)		1
9	SP1530457	TRUNK SUPPORT, LE, MULTI ADJ. SWINGAWAY, CPL. (ATTACHMENT INCL.)		1
10	SP1530456	TRUNK SUPPORT, RI, MULTI ADJ. SWINGAWAY, CPL. (ATTACHMENT INCL.)		1
11	SP1445799	TRUNK SUPPORT		1
12	1485420	TRUNK SUPPORT, LE, EXTENDED		1
13	1485419	TRUNK SUPPORT, RI, EXTENDED		1
14	SP1444207	TRUNK SUPPORT, TR26, CPL.		1
15	1485418	TRUNK SUPPORT, LE, MULTI ADJ. EXTENDED		1
16	1485417	TRUNK SUPPORT, RI, MULTI ADJ. EXTENDED		1
17	1451822	Lateral support, multi adjustable - LH		1
18	1451821	Lateral support, multi adjustable - RH		1
19	SP1529225	Trunk support, multi adj., left		1
20	SP1529224	Trunk support, multi adj., right		1
21	5253170	Lateral Support Bracket		1
22	1524903	Bracket LH/RH		1
23	SP1448728	Memory, Trunk support		1
24	1437642	Trunk Protection Dartex		1

Trunk supports, Azalea MAX

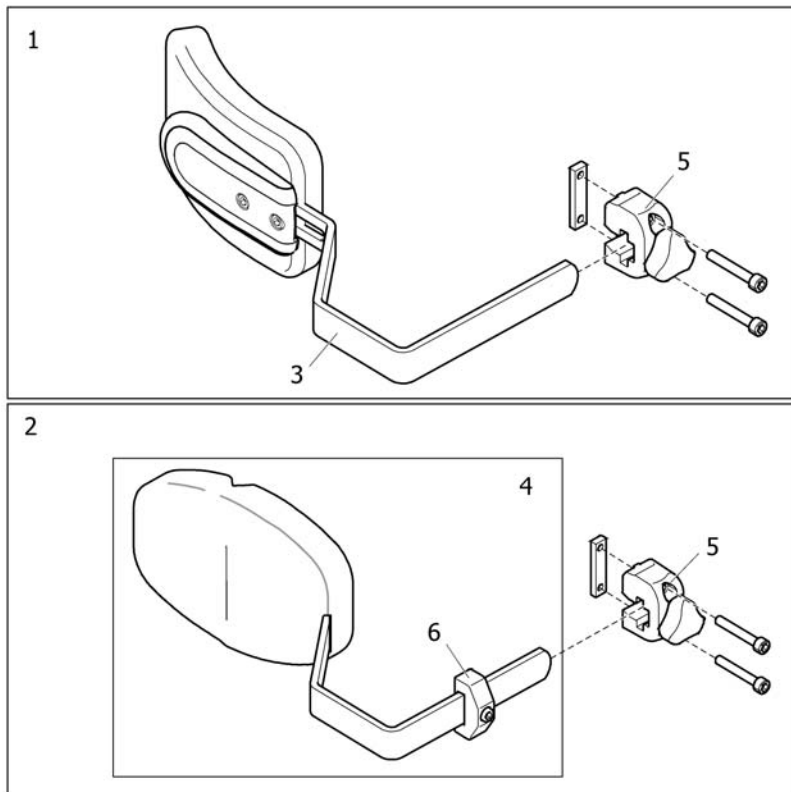


Pos.	Part number	Description	Valid from → until	Qty
1	SP1563735	TRUNK SUPPORT, FIXED, AZALEA MAX		1
2	SP1563737	TRUNK SUPPORT, MULTIADJ. AZALEA MAX		1
3	SP1563734	TRUNK SUPPORT 2012, MAX		1
4	SP1563736	TRUNK SUPPORT, MAX		1
5	5253170	Lateral Support Bracket		1
6	1448728	Memory, Trunk support		1

Trunk supports, Azalea Minor

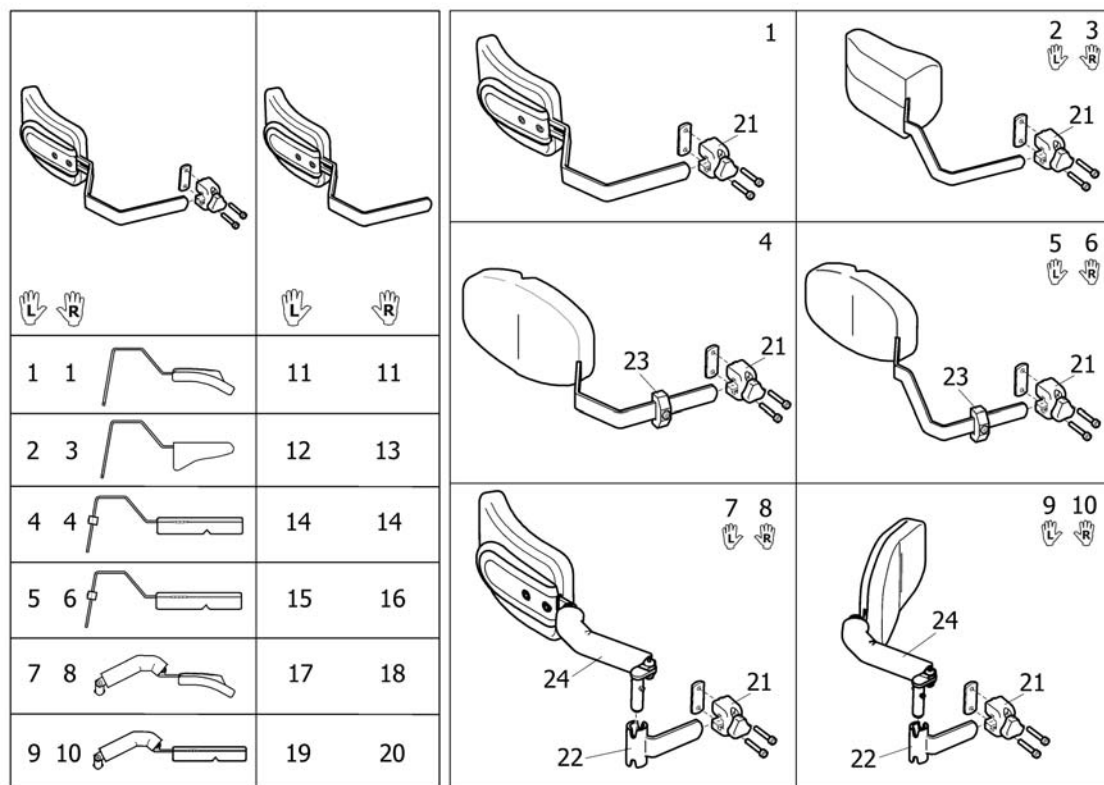


Pos.	Part number	Description	Valid from → until	Qty
1	SP1551850	TRUNK SUPPORT 2012, CPL, INCL. ATTACHMENT, AZALEA MINOR		1
2	SP1492608	TRUNK SUPPORT, LE, EXTENDED, CPL. (ATTACHMENT INCL.)		1
3	SP1492609	TRUNK SUPPORT, RI, EXTENDED, CPL. (ATTACHMENT INCL.)		1
4	SP1448692	TRUNK SUPPORT, MULTI ADJ. CPL. (ATTACHMENT INCL.)		1
5	SP1484447	TRUNK SUPPORT, LE, MULTI ADJ. EXTENDED, CPL. (ATTACHMENT INCL.)		1
6	SP1484448	TRUNK SUPPORT, RI, MULTI ADJ. EXTENDED, CPL. (ATTACHMENT INCL.)		1
7	SP1551851	TRUNK SUPPORT, LE, SWINGAWAY, AZALEA MINOR		1
8	SP1551852	TRUNK SUPPORT, RI, SWINGAWAY, AZALEA MINOR		1
9	SP1530457	TRUNK SUPPORT, LE, MULTI ADJ. SWINGAWAY, CPL. (ATTACHMENT INCL.)		1
10	SP1530456	TRUNK SUPPORT, RI, MULTI ADJ. SWINGAWAY, CPL. (ATTACHMENT INCL.)		1
11	1557653	TRUNK SUPPORT 2012, AZALEA MINOR		1
12	1485420	TRUNK SUPPORT, LE, EXTENDED		1
13	1485419	TRUNK SUPPORT, RI, EXTENDED		1
14	SP1444207	TRUNK SUPPORT, TR26, CPL.		1
15	1485418	TRUNK SUPPORT, LE, MULTI ADJ. EXTENDED		1
16	1485417	TRUNK SUPPORT, RI, MULTI ADJ. EXTENDED		1
17	1551853	Lh trunk-support cplt		1
18	1551866	Rh trunk-support complet		1
19	SP1529225	Trunk support, multi adj., left		1
20	SP1529224	Trunk support, multi adj., right		1
21	5253170	Lateral Support Bracket		1
22	1524903	Bracket LH/RH		1
23	1448728	Memory, Trunk support		1
24	1437642	Trunk Protection Dartex		1

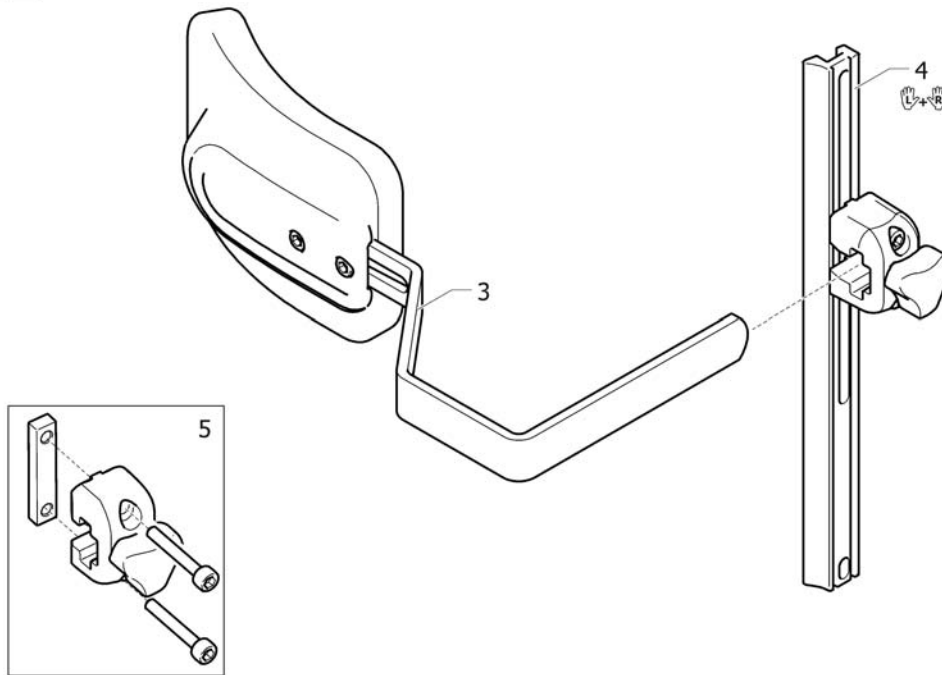

SPS1552651

Pos.	Part number	Description	Valid from → until	Qty
1	1552651	Retractable left trunk support, adjustable in width and height Dartex		1
2	1552652	Retractable right trunk support, adjustable in width and height Dartex		1
3	1552698	Trunk support		1
4	1518526	Receiver trunk support (pair)		1
5	1543574	Locking trunk support		1


Attachment, Positioning Belt

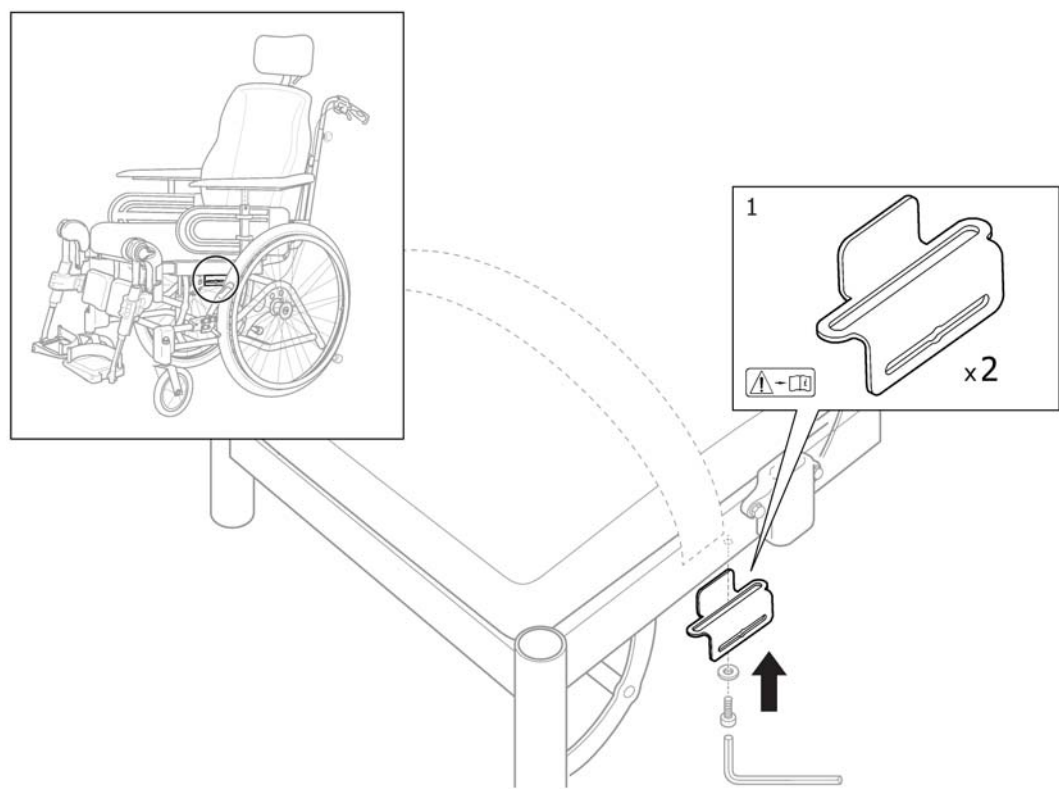

SPS1563952

Pos.	Part number	Description	Valid from → until	Qty
1	SP1563952	ATTACHMENT KIT, POSITIONING BELT, AZALEA		1


1

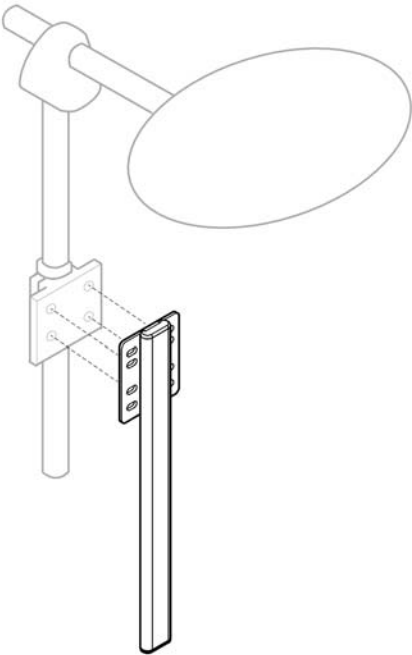

SP1581931

Pos.	Part number	Description	Valid from → until	Qty
1	SP1581931	Alternative Headrest mounting bracket		1


Table tray

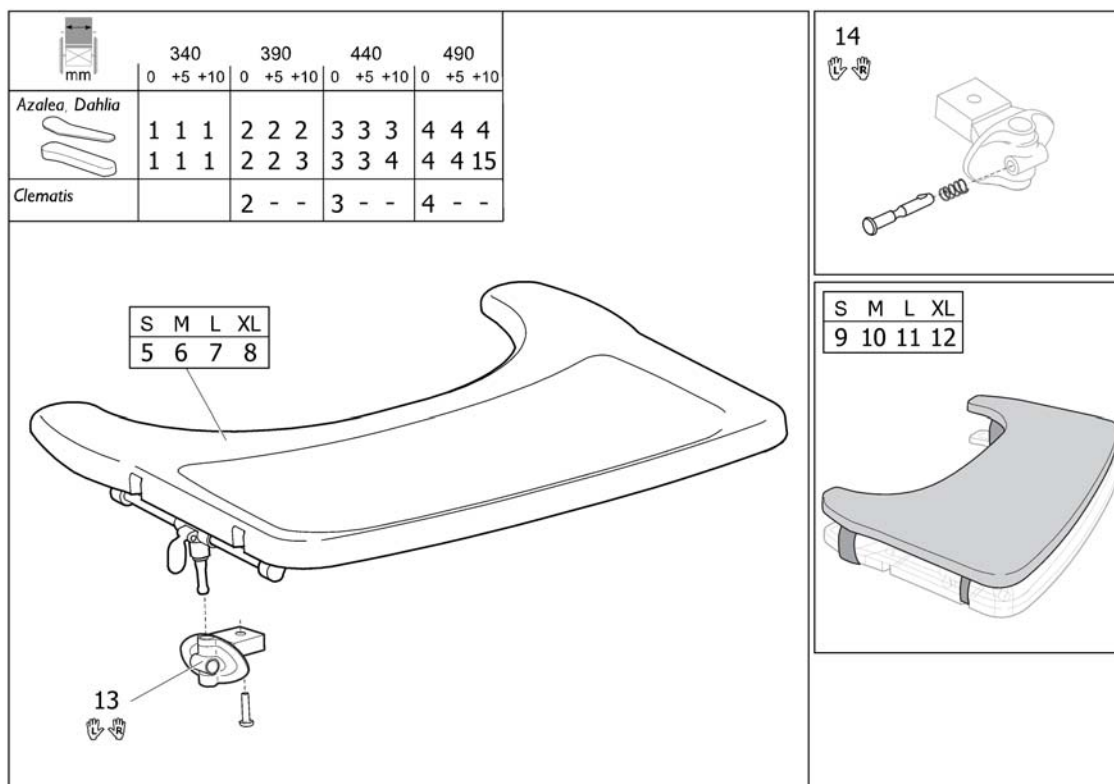

Pos.	Part number	Description	Valid from → until	Qty
1	SP1570747	TABLE TRAY SMALL INCL. MOUNTING KIT		1
2	SP1570748	Table tray MEDIUM incl. mounting kit		1
3	SP1570749	Table tray LARGE incl. mounting kit		1
4	SP1570775	Table tray X-LARGE incl. mounting kit		1
5	SP1567517	TABLE TRAY SMALL		1
6	SP1567518	Table tray Medium		1
7	SP1567519	Table tray Large		1
8	SP1568862	Table tray X-Large		1
9	SP1570746	TABLE CUSHION, SMALL		1
10	SP1571269	TABLE CUSHION, MEDIUM		1
11	SP1571270	TABLE CUSHION, LARGE		1
12	SP1571271	TABLE CUSHION, X-LARGE		1
13	SP1570745	Mounting Kit for table (pair)		1
14	SP1572460	Locking device for comfort tray (pair)		1
15	-	Table tray, Azalea Max		1

Table tray, Azalea Max

1


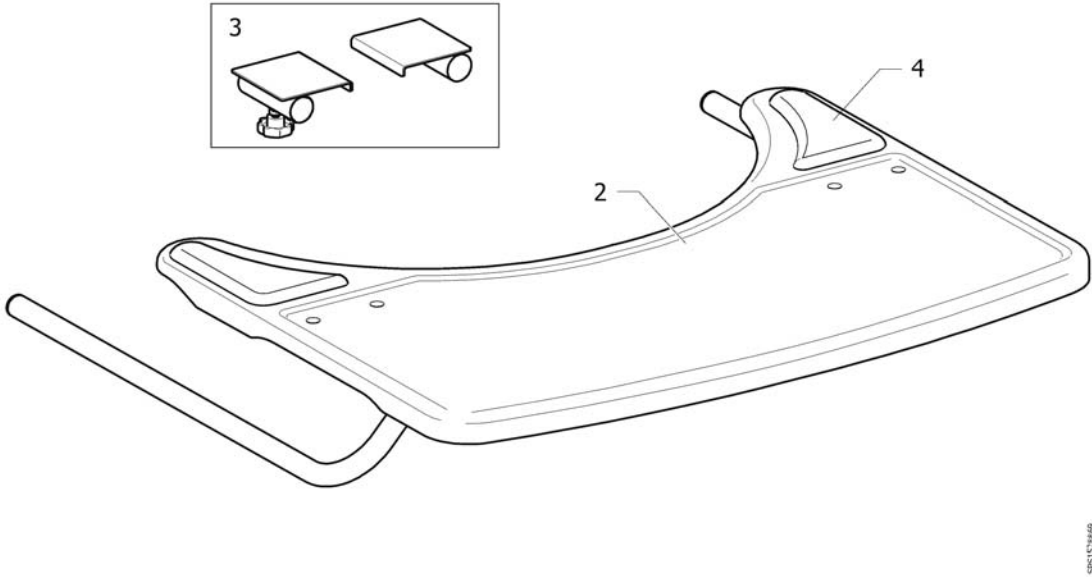


995152869

Pos.	Part number	Description	Valid from → until	Qty
1	1528869	Table Tray Cpl, Azalea Max		1
2	1531907	Table Tray, Azalea Max		1
3	1532190	Attachment kit, Table Tray, Azalea Max		1
4	1497825	Elbow pad pair for table tray		1

Table tray with armpad


mm	340			390			440			490		
	0	+5	+10	0	+5	+10	0	+5	+10	0	+5	+10
	1	1	1	2	2	2	3	3	3	4	4	4
	1	1	1	2	2	3	3	3	4	4		

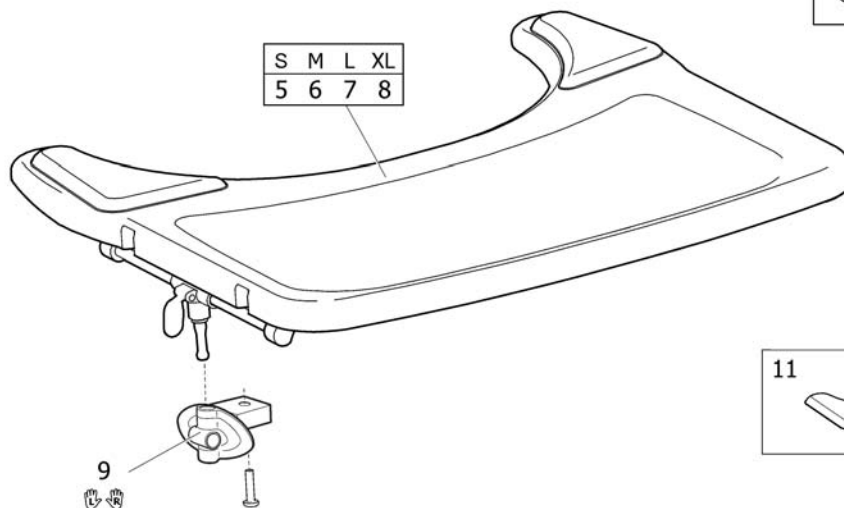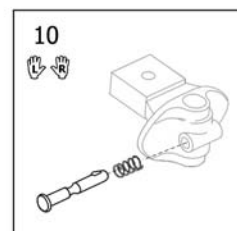

SP158087

Pos.	Part number	Description	Valid from → until	Qty
1	SP1580891	TABLE TRAY, S, ARMPAD, INCL. MOUNTING KIT		1
2	SP1580892	TABLE TRAY, M, ARMPAD, INCL. MOUNTING KIT		1
3	SP1580893	TABLE TRAY, L, ARMPAD, INCL. MOUNTING KIT		1
4	SP1580894	TABLE TRAY, XL, ARMPAD, INCL. MOUNTING KIT		1
5	SP1580887	TABLE TRAY, S, ARMPAD		1
6	SP1580888	TABLE TRAY, M, ARMPAD		1
7	SP1580889	TABLE TRAY, L, ARMPAD		1
8	SP1580890	TABLE TRAY, XL, ARMPAD		1
9	SP1570745	Mounting Kit for table (pair)		1
10	SP1572460	Locking device for comfort tray (pair)		1
11	1497825	Elbow pad pair for table tray		1

Table tray "Swingaway"

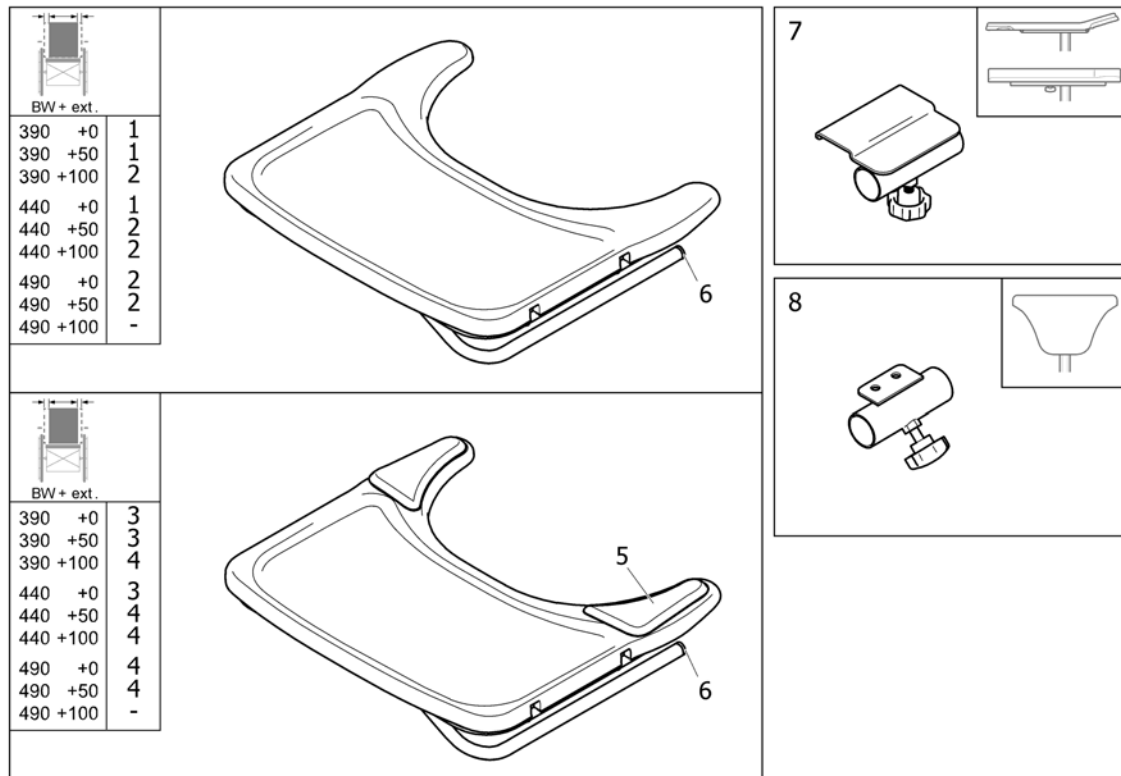


ISPS1449217

Pos.	Part number	Description	Valid from → until	Qty
1	SP1586424	Table tray Swingaway, small, cpl.		1
2	SP1586425	Table tray Swingaway, medium, cpl.		1
3	SP1584394	Table tray Swingaway incl. pads, small, cpl.		1
4	SP1584395	Table tray Swingaway incl. pads, medium, cpl.		1
5	1497825	Elbow pad pair for table tray		1
6	5716296	Plug Ø22x19.5, plastic black		1
7	1449218	ATTACHMENT, TABLE TRAY CPL., "SWINGAWAY"		1
8	1449839	ATTACHM. TABLE		1

Table tray "Swingaway", Azalea Minor

	340	+50	+100
SW(mm)	1	1	4

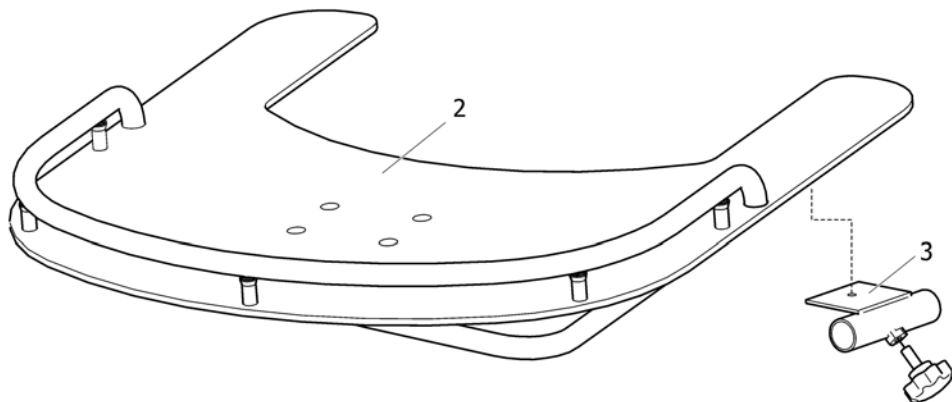


09/2011/06

Pos.	Part number	Description	Valid from → until	Qty
1	1509919	TABLE TRAY 340 KPL, AZALEA MINOR		1
2	1511916	Action 3 Junior Transparent Tray w 355 /380 mm		1
3	1509850	ATTACHMENT CPL., TABLE TRAY, AZALEA MINOR		1
4	-	Table tray "Swingaway"		1

Pushbar adj. Azalea Minor

1

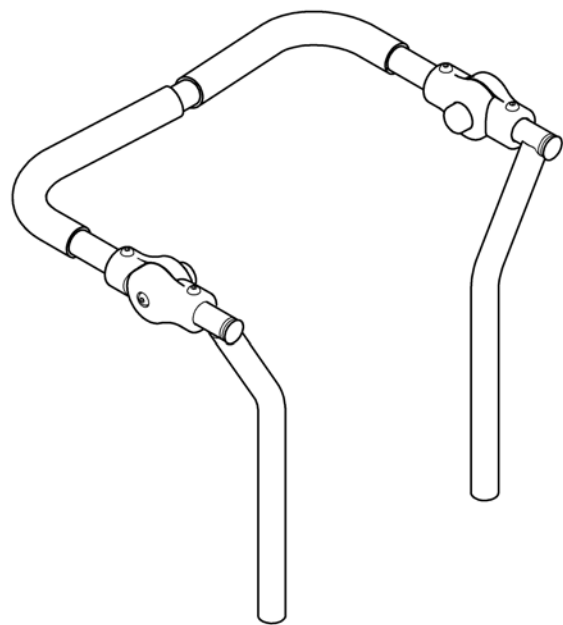


SP1563968

Pos.	Part number	Description	Valid from → until	Qty
1	SP1563968	PUSHBAR ADJ. AZALEA MINOR		1

Pushbar adj. Azalea

		
SW (mm)	mm	
390	362	1
440	412	2
490	463	3

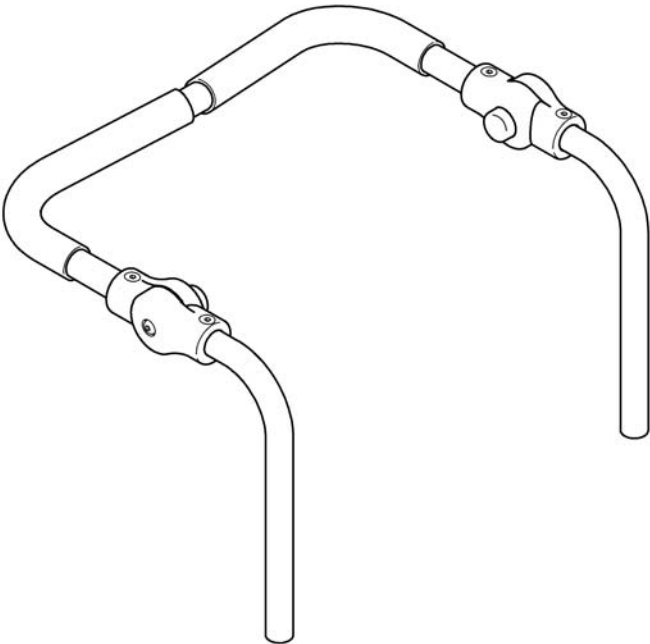


SP1563967

Pos.	Part number	Description	Valid from → until	Qty
1	SP1563967	PUSHBAR ADJ. 390		1
2	SP1564551	PUSHBAR ADJ. 440		1
3	SP1564552	PUSHBAR ADJ. 490		1

Push bar, Flex2


		
SW (mm)		
Bellis	410 (S)	2
	460 (M)	3
	510 (L)	4
Azalea	390	1
	440	2
	490	3

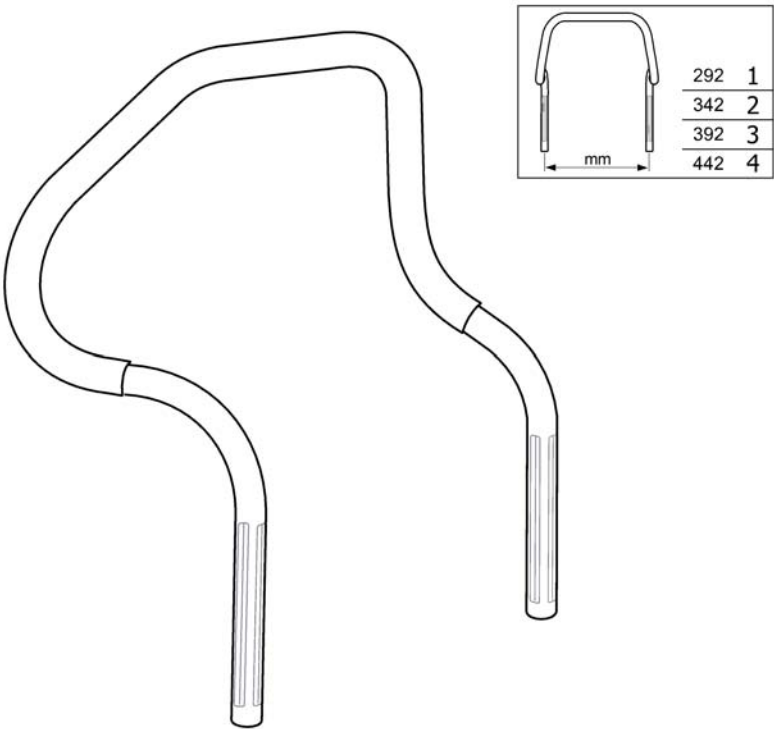

Pos.	Part number	Description	Valid from → until	Qty
1	1444350	PUSH BAR 292		1
2	5255136	PUSH BAR 342		1
3	5255137	PUSH BAR 392		1
4	5255138	PUSH BAR 442		1



Push handles, connected, Flex2

SW (mm)	
Bellis	
410	3
460	4
510	5
Azalea	
340	1
390	2
440	3
490	4

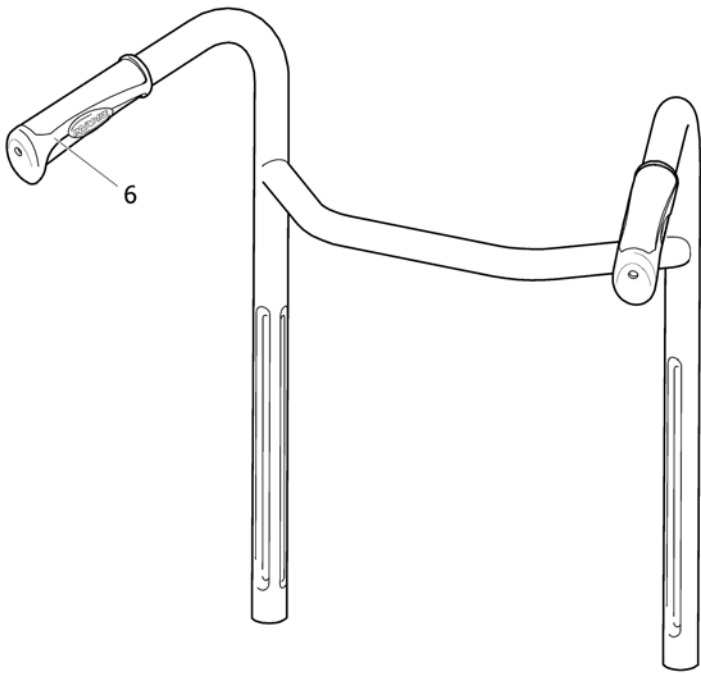

SPS1442006

Pos.	Part number	Description	Valid from → until	Qty
1	1508953-0509	PUSH HANDLE 34 CPL, BRACED		1
2	1420007-0509	PUSHHANDLE BRACED 314 (XS), CPL.		1
3	1420008-0509	PUSHH. BRACED C44,SMA BE		1
4	1420009-0509	PUSHH. BRACED C49,MED BE		1
5	1420010-0509	PUSHHANDLE BRACED LA,BEL		1
6	SP1537634	HANDLE 22MM, PAIR		1

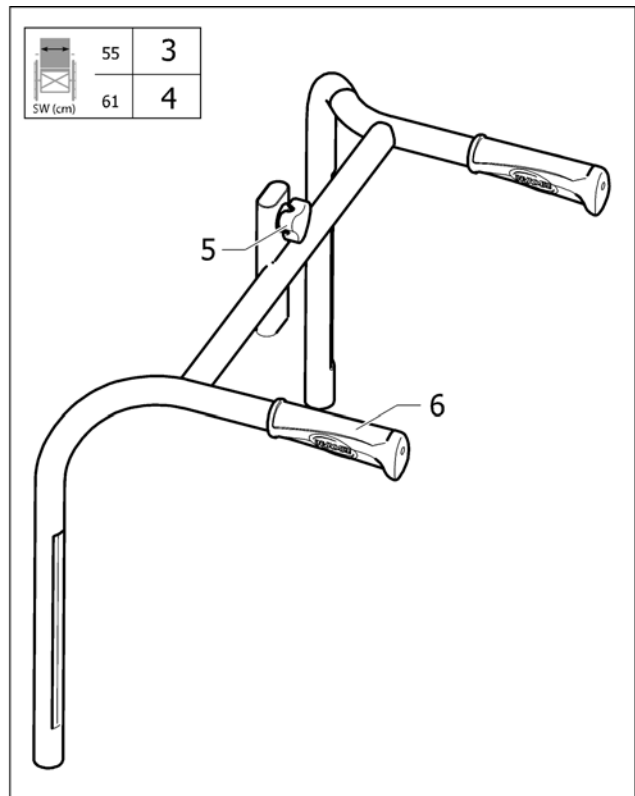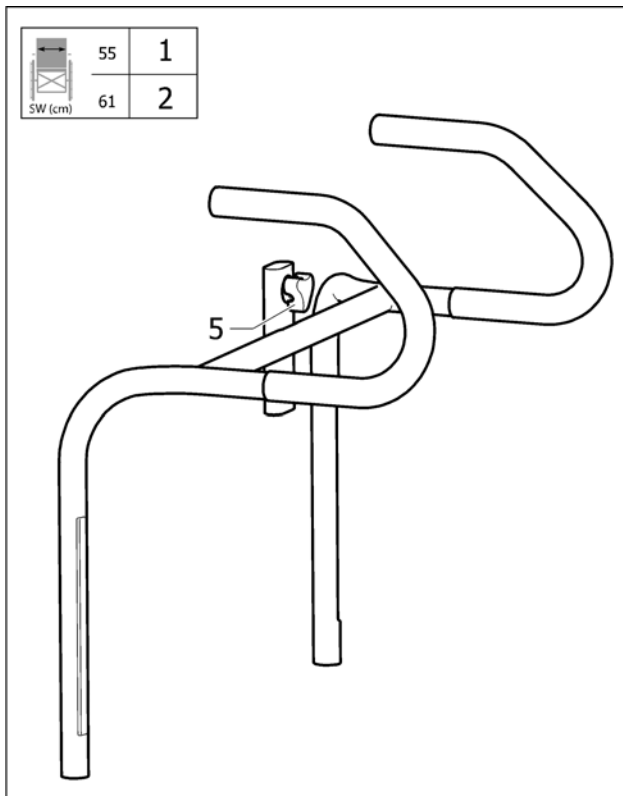
Push handles, Azalea Max

SP1523946

Pos.	Part number	Description	Valid from → until	Qty
1	1523947	High Push Bar 55		1
2	1523946	High Push Bar 61		1
3	1524533	Push handle braced 55		1
4	1524529	Push handle braced 61		1
5	1444175	HANDWHEEL UNLOSEABLE CPL CPL		1
6	SP1537634	HANDLE 22MM, PAIR		1

Crutch holder, Flex3 backrest


1

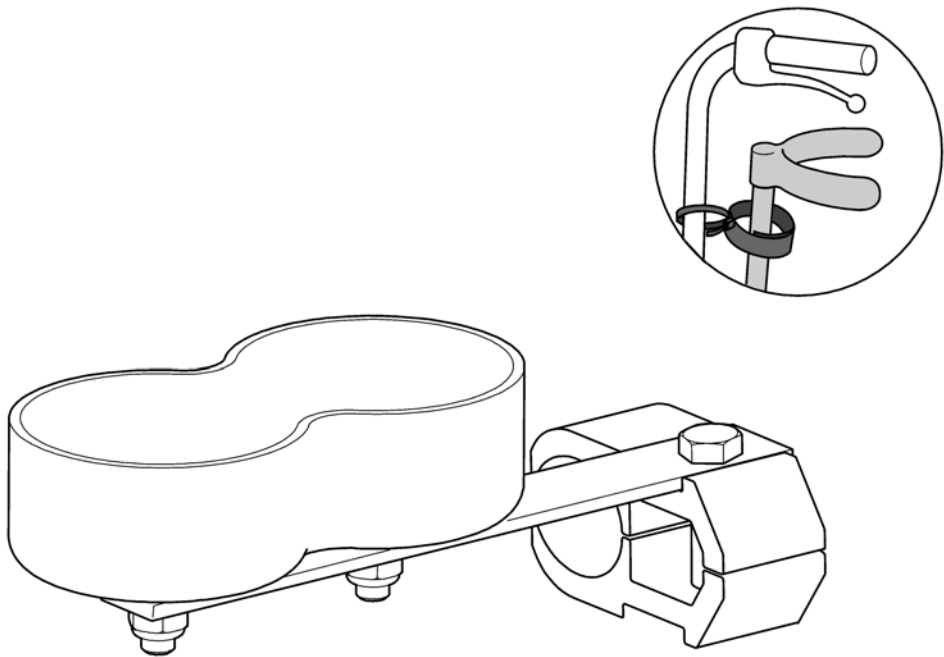

SPS1560811

Pos.	Part number	Description	Valid from → until	Qty
1	SP1560811	Crutch holder		1


1

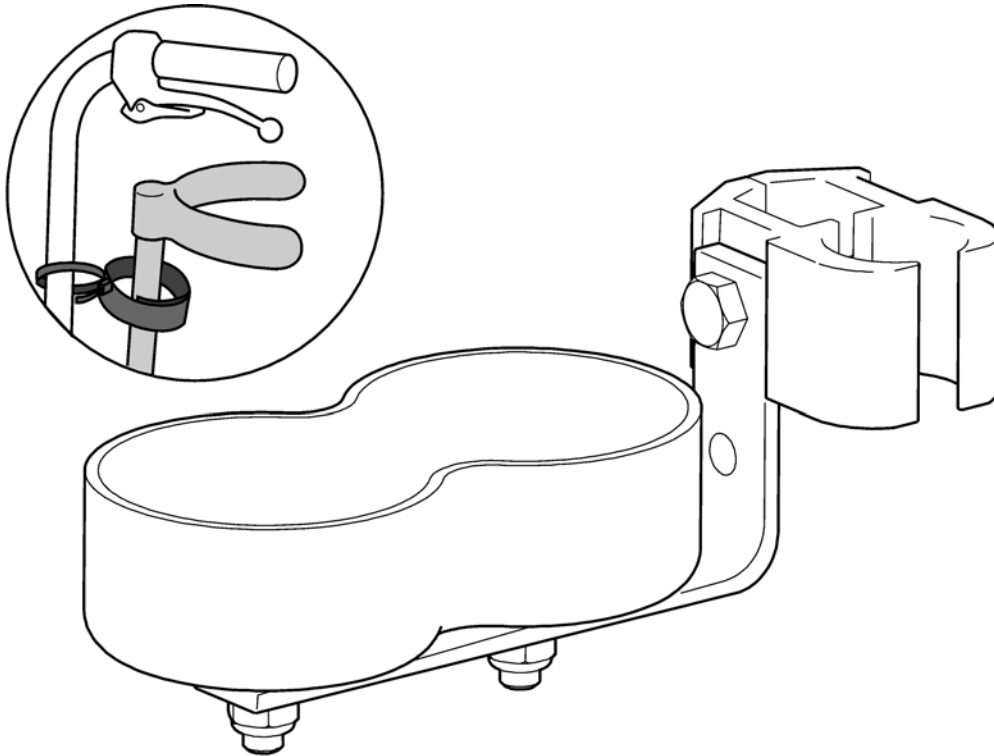

SP55295015


Pos.	Part number	Description	Valid from → until	Qty
1	5295015	CRUTCH HOLDER		1

Wash sealing kit

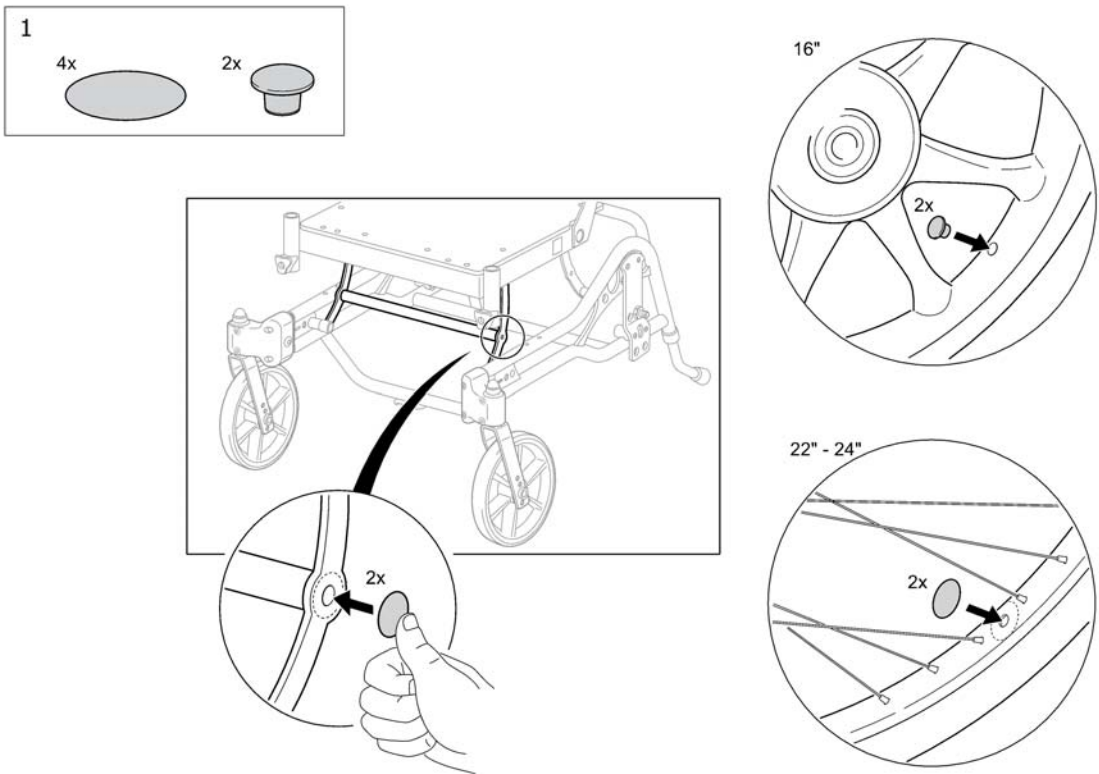


Pos.	Part number	Description	Valid from → until	Qty
1	SP1563953	WASH SEALING KIT		1

Tilt scale

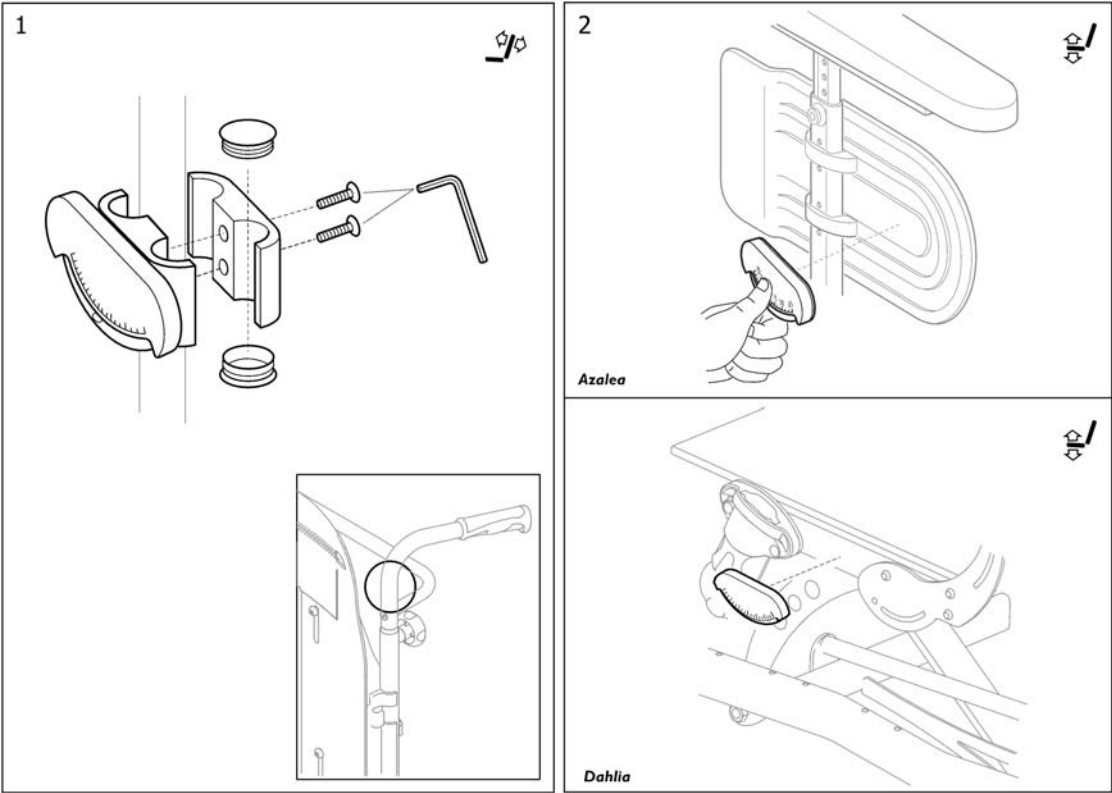


Pos.	Part number	Description	Valid from → until	Qty
1	SP1563954	Tilt scale, backrest		1
2	SP1561584	Tilt scale, seat		1

Clinic kit

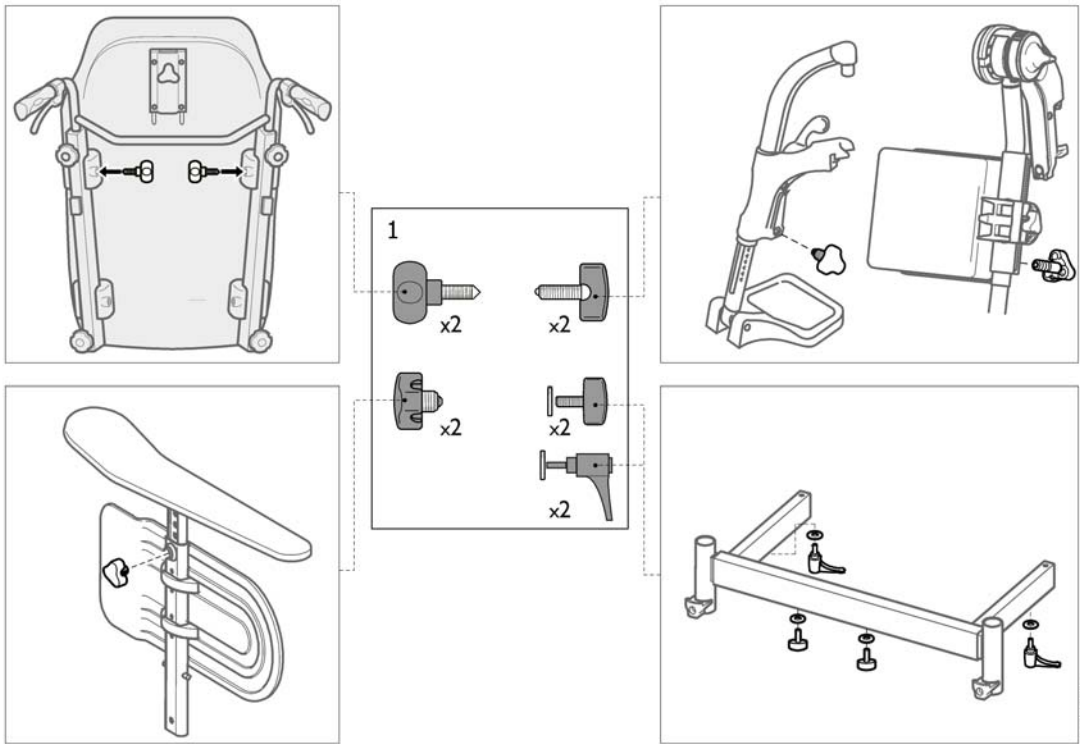

Pos.	Part number	Description	Valid from → until	Qty
1	SP1484457	CLINIC KIT, AZALEA/DAHLIA		1


16" + 22"	
Azalea	1
Azalea Base	
Azalea Minor	

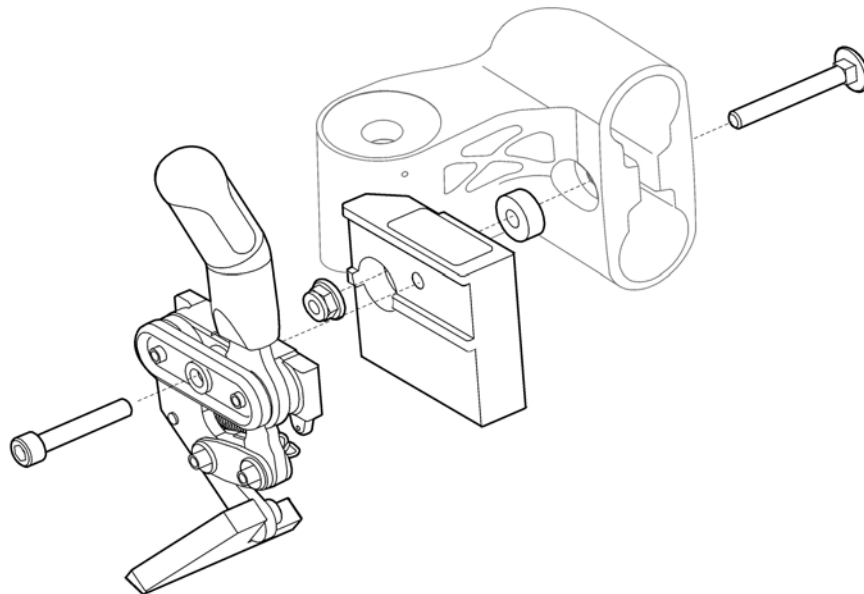

SP1564433

Pos.	Part number	Description	Valid from → until	Qty
1	SP1564433	DIRECTION LOCK FOR CASTORS, 16+22" WHEELS		1

Dripstand

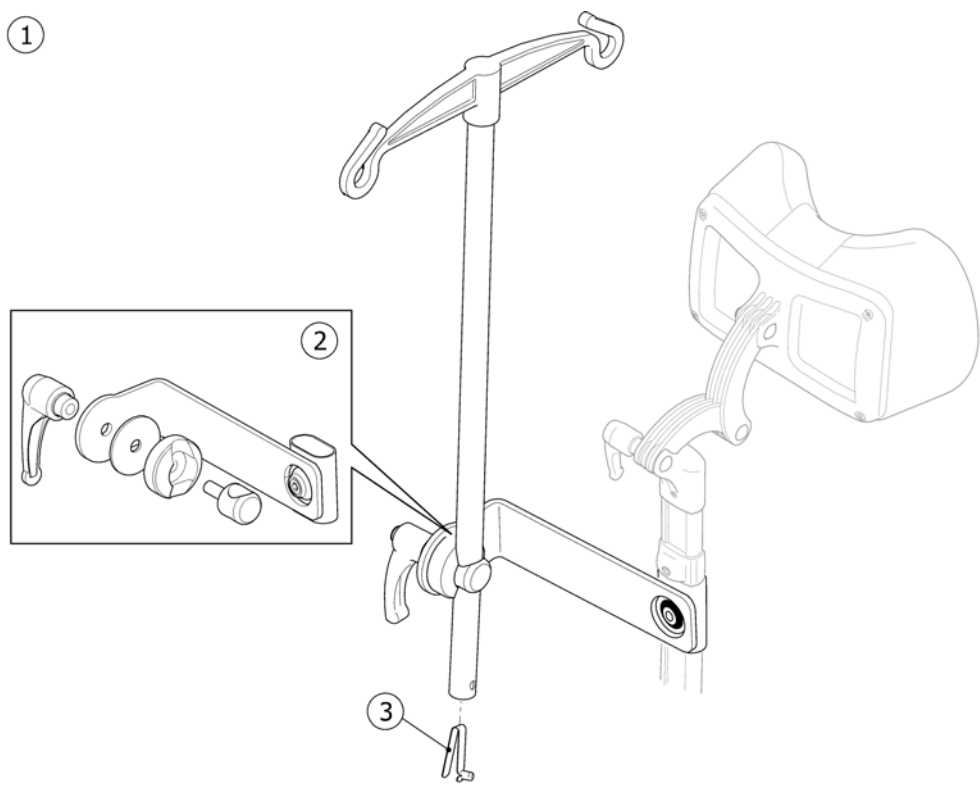

SPS1440897

Pos.	Part number	Description	Valid from → until	Qty
1	1440897	Dripstand + Fixation		1
2	1518781	ATTACHMENT, DRIP STAND		1
3	1560734	Spring pin locked armrest		1



Yes, you can.®



Belgium & Luxemburg:

Invacare nv, Autobaan 22, B-8210 Loppem
Tel: (32) (0)50 83 10 10, Fax: (32) (0)50 83 10 11
belgium@invacare.com / www.invacare.be

Denmark:

Invacare A/S, Sdr. Ringvej 37, DK-2605 Brøndby
Tel: (45) (0)36 90 00 00, Fax: (45) (0)36 90 00 01
denmark@invacare.com / www.invacare.dk

Deutschland:

Invacare Aquatec GmbH, Alemannenstraße 10, D-88316 Isny
Tel: (49) (0)75 62 7 00 0, Fax: (49) (0)75 62 7 00 66
info@invacare-aquatec.com / www.invacare-aquatec.de

European Distributor Organisation:

Invacare, Kleiststraße 49, D-32457 Porta Westfalica
Tel: (49) (0)57 31 754 540, Fax: (49) (0)57 31 754 541
edo@invacare.com / www.invacare.eu.com

España:

Invacare SA, c/Areny s/n, Polígon Industrial de Celrà, E-17460 Celrà (Girona)
Tel: (34) (0)972 49 32 00, Fax: (34) (0)972 49 32 20
contactsp@invacare.com / www.invacare.es

France:

Invacare Poirier SAS, Route de St Roch, F-37230 Fondettes
Tel: (33) (0)2 47 62 64 66, Fax: (33) (0)2 47 42 12 24
contactfr@invacare.com / www.invacare.fr

Ireland:

Invacare Ireland Ltd, Unit 5 Seatown Business Campus, Seatown Road, Swords, County Dublin - Ireland
Tel: (353) 1 810 7084, Fax: (353) 1 810 7085
ireland@invacare.com / www.invacare.ie

Italia:

Invacare Mecc San s.r.l., Via dei Pini 62, I-36016 Thiene (VI)
Tel: (39) 0445 38 00 59, Fax: (39) 0445 38 00 34
italia@invacare.com / www.invacare.it

Nederland:

Invacare BV, Celsusstraat 46, NL-6716 BZ Ede
Tel: (31) (0)318 695 757, Fax: (31) (0)318 695 758
nederland@invacare.com, csede@invacare.com / www.invacare.nl

Norge:

Invacare AS, Grensesvingen 9, Postboks 6230, Etterstad, N-0603 Oslo
Tel: (47) (0)22 57 95 00, Fax: (47) (0)22 57 95 01
norway@invacare.com, island@invacare.com / www.invacare.no

Österreich:

Invacare Austria GmbH, Herzog Odilostrasse 101, A-5310 Mondsee
Tel: (43) 6232 5535 0, Fax: (43) 6232 5535 4
info-austria@invacare.com / www.invacare.at

Portugal:

Invacare Lda, Rua Estrada Velha, 949, P-4465-784 Leça do Balio
Tel: (351) (0)225 1059 46/47, Fax: (351) (0)225 1057 39
portugal@invacare.com / www.invacare.pt

Sverige & Suomi:

Invacare AB, Fagerstagatan 9, S-163 91 Spånga
Tel: (46) (0)8 761 70 90, Fax: (46) (0)8 761 81 08
sweden@invacare.com, finland@invacare.com / www.invacare.se

Switzerland:

Invacare AG, Benkenstrasse 260, CH-4108 Witterswil
Tel: (41) (0)61 487 70 80, Fax: (41) (0)61 487 70 81
switzerland@invacare.com / www.invacare.ch

United Kingdom:

Invacare Limited, Pencoed Technology Park, Pencoed, Bridgend CF35 5AQ
Switchboard Tel: (44) (0)1656 776200, Fax: (44) (0)1656 776201
Customer services Tel: (44) (0)1656 776222, Fax: (44) (0)1656 776220
uk@invacare.com / www.invacare.co.uk