

Spare Parts Catalogue

Version: 2008/09/29a EN



Invacare® **Action®³ Junior®**



Yes, you can.®

General Information

For any order of spare parts the following information is required:

- Product type*
- Product name & model*
- Serial Number*
- Part-Number*
- Part-Description*

Exploded view drawings and code numbers are not contractual and subject to modification, in relation to Invacare® continuous product improvement policy to meet end users needs.

The reproduction or use of this work, or of any substantial part of this work, in any form or by any electronic, mechanical or other means, including photocopying, scanning and recording, and in any information storage and retrieval system, including electronic transmission, is forbidden without the prior written permission of Invacare® International Sarl.



Yes, you can.®

TABLE OF CONTENTS

Chassis

Side Frames SW 20,5 to 28 cm	5
Side Frames SW 30,5 to 38 cm	6
Stabilizer Side Frames SW 20,5 to 28 cm	7
Stabilizer Side Frames SW 30,5 to 38 cm	8
Folding System Fixed	9
Adjustable Folding System	10

Seat

Seat Upholstery	11
Padded Seats Upholstery	12
Depth Adjustable Seat Upholstery	13
Multi Adjustable Seat Upholstery	14

Backrest

Fixed Backrest	15
Fixed Backrest, Height Adjustable, Push Handle	16
Recliner Backrest 0° To 30° Racket	17
Recliner Backrest 0° to 30° Gas Spring	18
Fold Down Backrest	19
Adjustable Backrest Upholstery	20
Tension adjustable backrest upholstery	21
Backrest extension	22

Legrests

Front Hanger 80° & 90°	23
Elevating Legrest with angle adjustable footrest	24
Elevating Legrest with angle and depth adjustable footrest	25

Footrests

Standard Footplate	26
Adjustable Footplate For 80° Hanger	27
Adjustable Footplate For 90° Hanger and Stabilizer Frame	28

Siderests

Height Adjustable Armrest N°1	29
Height Adjustable Armrest N°1 Junior	30
Height Adjustable Skirtguard Junior	31
Height Adjustable Mudguard	32
Armrest Evolutive <12/2007	33

Front Wheels

Fork < 12/2007	34
Fork	35
5" Castors	36

Rear Wheels



TABLE OF CONTENTS

20" Standard Wheels	37
20" Standard Wheels accessories	38
22" Standard Wheels > 03/2007	39
22" Standard Wheels < 03/2007	40
22" Standard Wheels accessories	41
22"Hub brake wheels with accessories > 03/2007	42
22"Hub brake wheels with accessories < 03/2007	43
Pump And Support	44

Brakes

Push Brakes : Standard And With Extension	45
Attendant Hub Brake	46
22"Hub brake wheels with accessories > 03/2007	42
22"Hub brake wheels with accessories < 03/2007	43

Safety Options

Rear Anti Tippers	47
Tipping Rear Anti Tipper	48
Belts	49
Transfer Kit	50
Reflector Kit	51

Seating Options

Cushion	52
Essential Junior Cushion	53
Headrest & Neckrest	54
Heel Strap - Calf Strap Junior	55
Back Brace & Push Bar	56

Propulsion Options


Dual Amputee Position	57
Transit Rear Wheel Pusch Brake > 12/2007	58
Transit Rear Wheel Hub Brake >12/2007	59
Spoke Guard	60

Others Options & Accessories

Tipping Tube - Back Bag	61
Transparent Tray < 10/2007	62
Junior Transparent Trays	63


Side Frames SW 20,5 to 28 cm

Dark Red	R :	1
	L :	2
Sapphire Blue	R :	3
	L :	4
Tangerine Orange	R :	5
	L :	6
Anthracite	R :	7
	L :	8
Bubble Gum Pink	R :	9
	L :	10

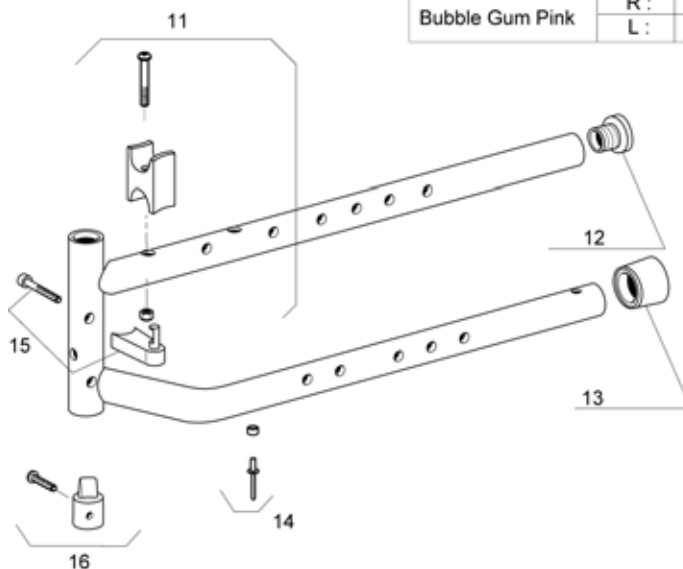


Pos.	Part number	Description	Qty
1	1511920-0607	Rh Side Frame A3 Junior Dark Red 20,5 to 28 cm	1
2	1511921-0607	Lh Side Frame A3 Junior Dark Red 20,5 to 28 cm	1
3	1511920-0608	Rh Side Frame A3 Junior Sapphire Blue 20,5 to 28 cm	1
4	1511921-0608	Lh Side Frame A3 Junior Sapphire Blue 20,5 to 28 cm	1
5	1511920-0613	Rh Side Frame A3 Junior Tangerine Orange 20,5 to 28 cm	1
6	1511921-0613	Lh Side Frame A3 Junior Tangerine Orange 20,5 to 28 cm	1
7	1511920-0609	Rh Side Frame A3 Junior Anthracite 20,5 to 28 cm	1
8	1511921-0609	Lh Side Frame A3 Junior Anthracite 20,5 to 28 cm	1
9	1511920-0614	Rh Side Frame A3 Junior Bubble Gum Pink 20,5 to 28 cm	1
10	1511921-0614	Lh Side Frame A3 Junior Bubble Gum Pink 20,5 to 28 cm	1
11	0572-06061	H Frame Rest Kit 3000	1
12	1418112	Crossbrace back cap	1
13	0021N05386	1" cap	1
14	1431304	Cross Brace Rivet & Washer Kit	1
15	0572-06910	Locking Hanger Device	1
16	1497559	Legrest Indexer >09/06	1

Side Frames SW 30,5 to 38 cm

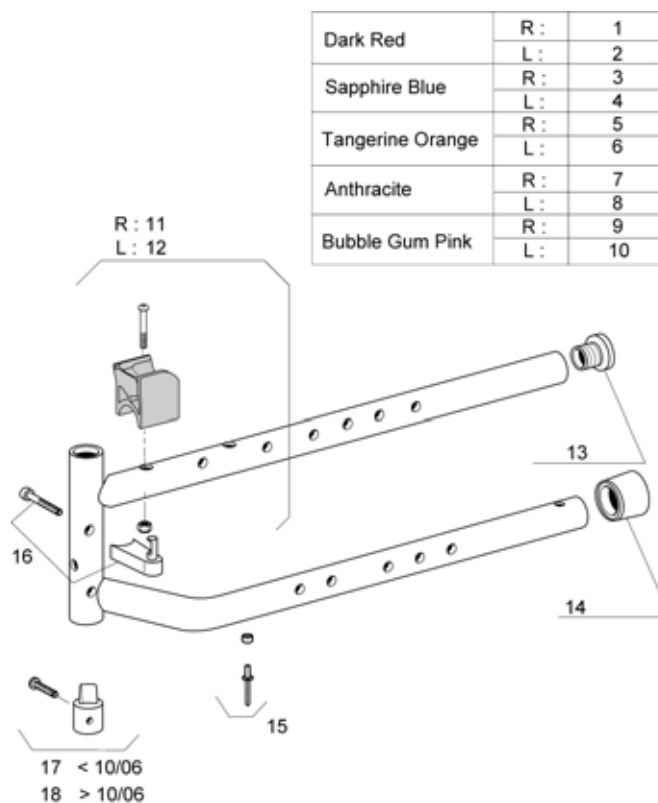


Pos.	Part number	Description	Qty
1	1489011-0607	Rh Side Frame A3 Junior Dark Red 30,5 to 38 cm	1
2	1489012-0607	Lh Side Frame A3 Junior Dark Red 30,5 to 38 cm	1
3	1489011-0608	Rh Side Frame A3 Junior Sapphire Blue 30,5 to 38 cm	1
4	1489012-0608	Lh Side Frame A3 Junior Sapphire Blue 30,5 to 38 cm	1
5	1489011-0613	Rh Side Frame A3 Junior Tangerine Orange 30,5 to 38 cm	1
6	1489012-0613	Lh Side Frame A3 Junior Tangerine Orange 30,5 to 38 cm	1
7	1489011-0609	Rh Side Frame A3 Junior Anthracite 30,5 to 38 cm	1
8	1489012-0609	Lh Side Frame A3 Junior Anthracite 30,5 to 38 cm	1
9	1489011-0614	Rh Side Frame A3 Junior Bubble Gum Pink 30,5 to 38 cm	1
10	1489012-0614	Lh Side Frame A3 Junior Bubble Gum Pink 30,5 to 38 cm	1
11	1491101	Cross Brace Support Kit Junior Rh	1
12	1491102	Cross Brace Support Kit Junior Lh	1
13	1418112	Crossbrace back cap	1
14	0021N05386	1" cap	1
15	1431304	Cross Brace Rivet & Washer Kit	1
16	0572-06910	Locking Hanger Device	1
17	1497559	Legrest Indexer >09/06	1

Stabilizer Side Frames SW 20,5 to 28 cm

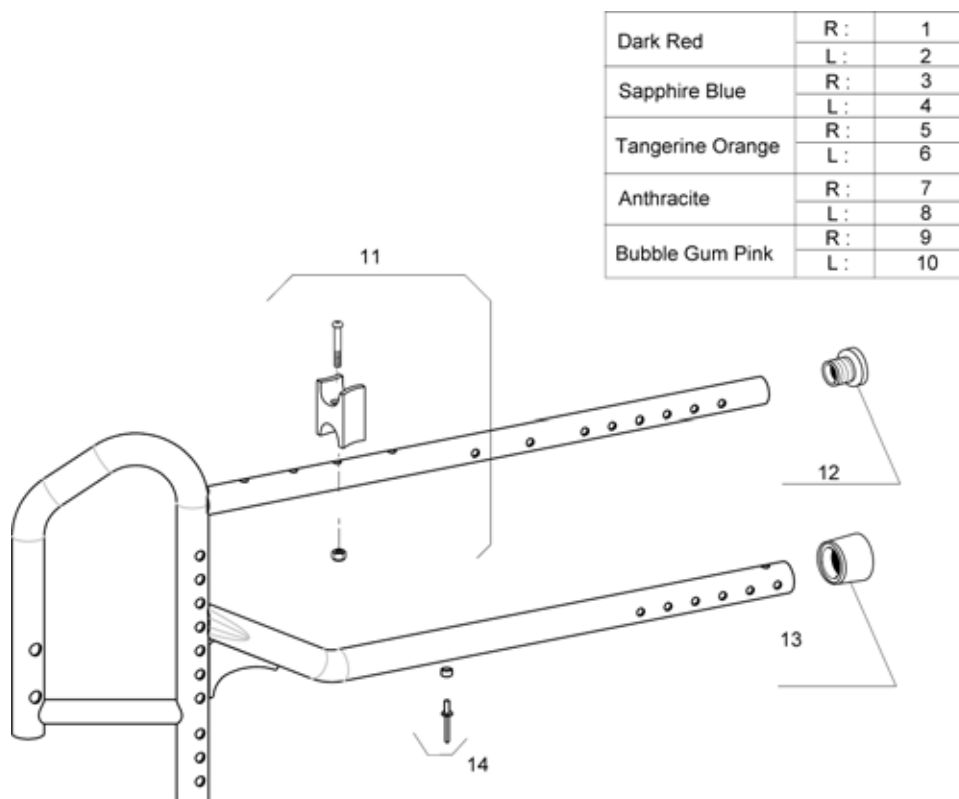


Pos.	Part number	Description	Qty
8	1512007-0609	Lh Stabilizer Side Frame A3 Junior Anthracite 20,5 to 28 cm	1
1	1512006-0607	Rh Stabilizer Side Frame A3 Junior Dark Red 20,5 to 28 cm	1
2	1512007-0607	Lh Stabilizer Side Frame A3 Junior Dark Red 20,5 to 28 cm	1
3	1512006-0608	Rh Stabilizer Side Frame A3 Junior Sapphire Blue 20,5 to 28 cm	1
4	1512007-0608	Lh Stabilizer Side Frame A3 Junior Sapphire Blue 20,5 to 28 cm	1
5	1512006-0613	Rh Stabilizer Side Frame A3 Junior Tangerine Orange 20,5 to 28 cm	1
6	1512007-0613	Lh Stabilizer Side Frame A3 Junior Tangerine Orange 20,5 to 28 cm	1
7	1512006-0609	Rh Stabilizer Side Frame A3 Junior Anthracite 20,5 to 28 cm	1
9	1512006-0614	Rh Stabilizer Side Frame A3 Junior Bubble Gum Pink 20,5 to 28 cm	1
10	1512007-0614	Lh Stabilizer Side Frame A3 Junior Bubble Gum Pink 20,5 to 28 cm	1
11	0572-06061	H Frame Rest Kit 3000	1
12	1418112	Crossbrace back cap	1
13	0021N05386	1" cap	1
14	1431304	Cross Brace Rivet & Washer Kit	1

Stabilizer Side Frames SW 30,5 to 38 cm

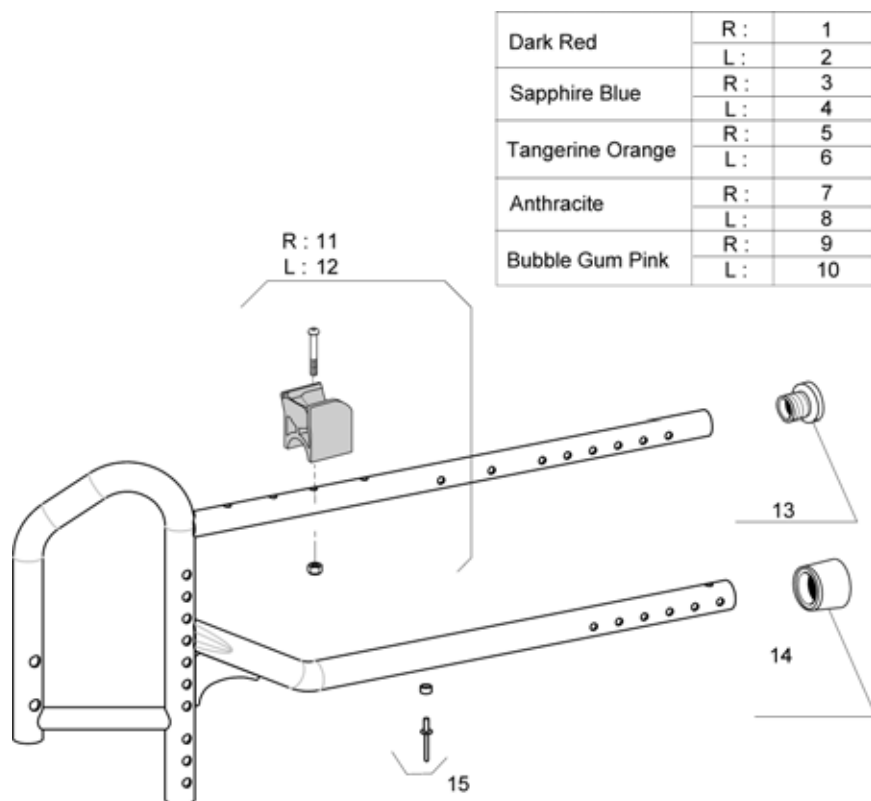


Pos.	Part number	Description	Qty
1	1511923-0607	Rh Stabilizer Side Frame A3 Junior Dark Red 30,5 to 38 cm	1
2	1511924-0607	Lh Stabilizer Side Frame A3 Junior Dark Red 30,5 to 38 cm	1
3	1511923-0608	Rh Stabilizer Side Frame A3 Junior Sapphire Blue 30,5 to 38 cm	1
4	1511924-0608	Lh Stabilizer Side Frame A3 Junior Sapphire Blue 30,5 to 38 cm	1
5	1511923-0613	Rh Stabilizer Side Frame A3 Junior Tangerine Orange 30,5 to 38 cm	1
6	1511924-0613	Lh Stabilizer Side Frame A3 Junior Tangerine Orange 30,5 to 38 cm	1
7	1511923-0609	Rh Stabilizer Side Frame A3 Junior Anthracite 30,5 to 38 cm	1
8	1511924-0609	Lh Stabilizer Side Frame A3 Junior Anthracite 30,5 to 38 cm	1
9	1511923-0614	Rh Stabilizer Side Frame A3 Junior Bubble Gum Pink 30,5 to 38 cm	1
10	1511924-0614	Lh Stabilizer Side Frame A3 Junior Bubble Gum Pink 30,5 to 38 cm	1
11	1491101	Cross Brace Support Kit Junior Rh	1
12	1491102	Cross Brace Support Kit Junior Lh	1
13	1418112	Crossbrace back cap	1
14	0021N05386	1" cap	1
15	1431304	Cross Brace Rivet & Washer Kit	1

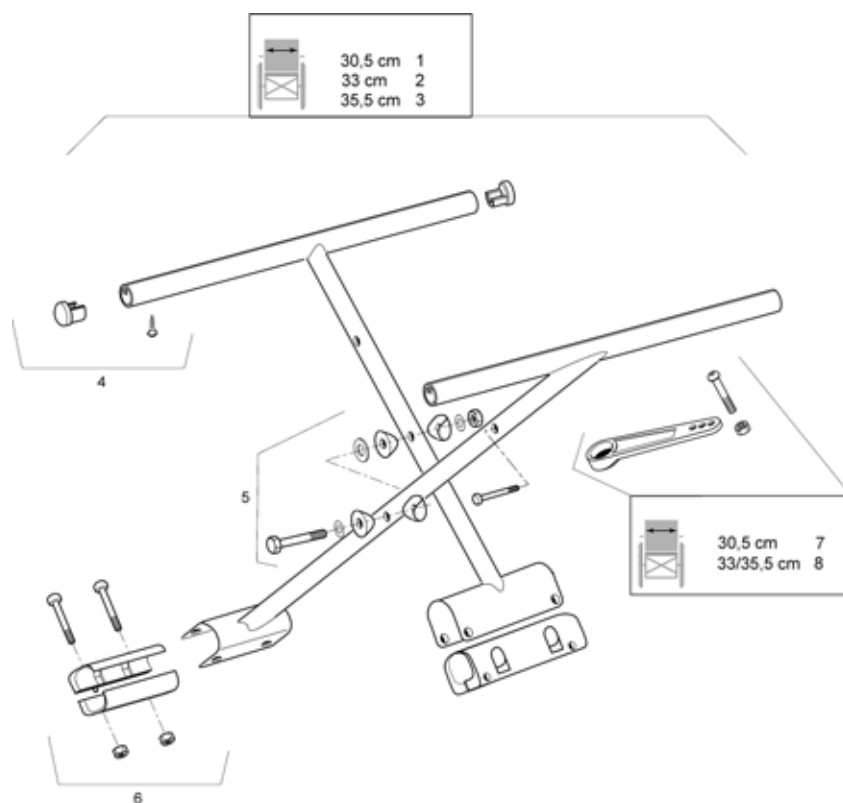Folding System Fixed


Pos.	Part number	Description	Qty
1	1478460-0601	Cross Brace Complete Mechanism W 30,5 x D 30 cm	1
2	1478461-0601	Cross Brace Complete Mechanism W 33 x D 35 cm	1
3	1478462-0601	Cross Brace Complete Mechanism W 35,5 x D 35 cm	1
4	1431307	Seat End Cap Kit	1
5	1431308	Cross Brace Axle Kit	1
6	1431305	Crossbrace Sleeve	1
7	1451295	Cross Brace Strut W 30 cm	1
8	1451296	Cross Brace Strut W 33/36 cm	1


Adjustable Folding System

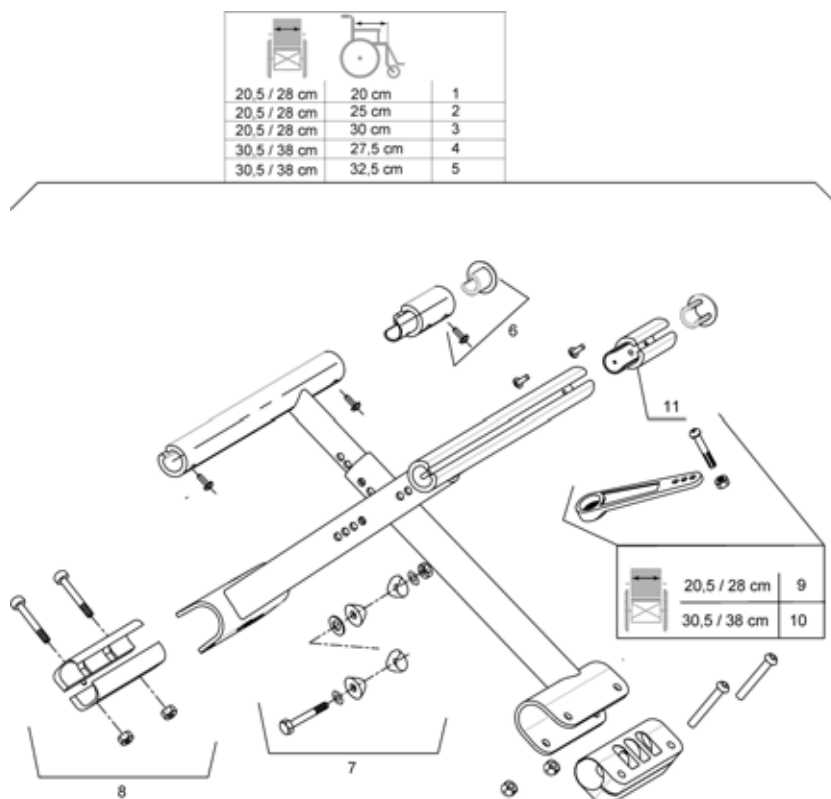

Pos.	Part number	Description	Qty
1	1511958-0601	Cross Brace Complete Adjustable Mechanism W 20,5/28 x D 20/25 cm	1
2	1511959-0601	Cross Brace Complete Adjustable Mechanism W 20,5/28 x D 25/30 cm	1
3	1513260-0601	Cross Brace Complete Adjustable Mechanism W 20,5/28 x D 30/35 cm	1
4	1512008-0601	Cross Brace Complete Adjustable Mechanism W 30,5/38 x D 27,5/32,5 cm	1
5	1512009-0601	Cross Brace Complete Adjustable Mechanism W 30,5/38 x D 32,5/37,5 cm	1
6	1431307	Seat End Cap Kit	1
7	1513307	Adjustable Cross Brace Axle Kit	1
8	1431305	Crossbrace Sleeve	1
9	1513305	Cross Brace Strut W 20,5/28 cm	1
10	1513306	Cross Brace Strut W 30,5/38 cm	1
11	1510022	5 cm Seat Extension Kit	1

Seat Upholstery

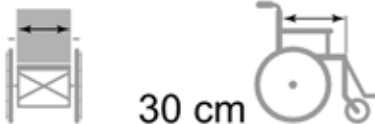
		
	30 cm	35 cm
	30,5 cm	1
	33 cm	-
	35,5 cm	-
		2
		3

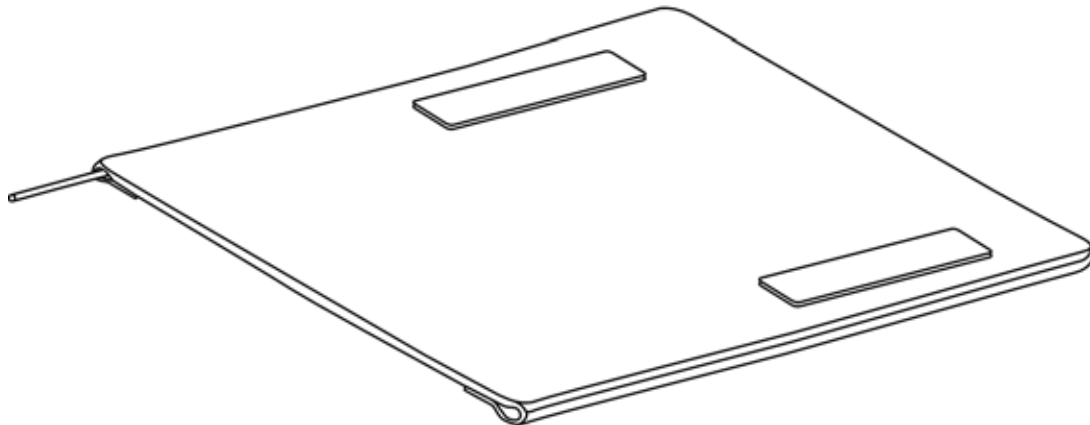

Pos.	Part number	Description	Qty
1	1491073	Standard Seat Upholstery W 30,5 cm X D 30 cm	1
2	1491074	Standard Seat Upholstery W 33 cm X D 35 cm	1
3	1491075	Standard Seat Upholstery W 35,5 cm X D 35 cm	1

Padded Seats Upholstery


	30 cm	35 cm
30,5 cm	1	-
33 cm	-	2
35,5 cm	-	3

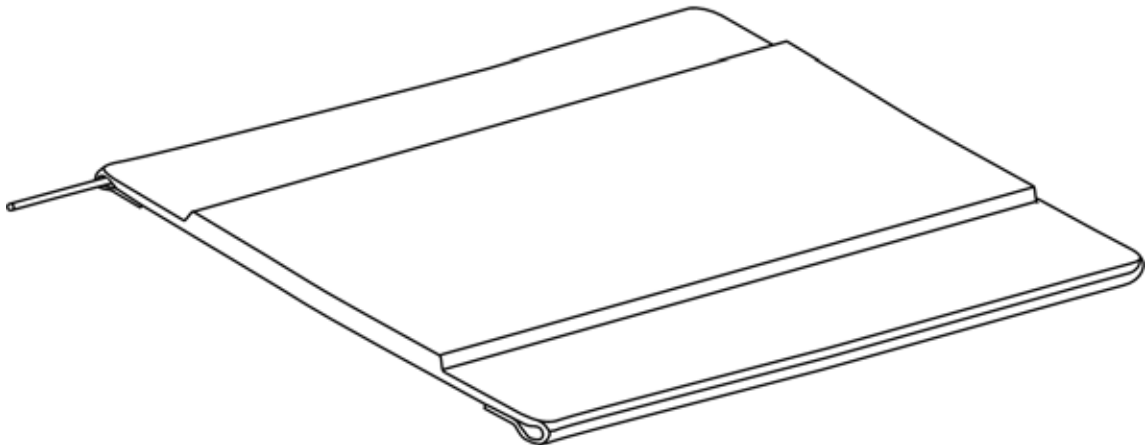

Pos.	Part number	Description	Qty
1	1489049	Padded Seat Upholstery W 30,5 cm X D 30 cm	1
2	1489050	Padded Seat Upholstery W 33 cm X D 35 cm	1
3	1489051	Padded Seat Upholstery W 35,5 cm X D 35 cm	1


Depth Adjustable Seat Upholstery


	
30,5 cm	1
33 cm	2
35,5 cm	3

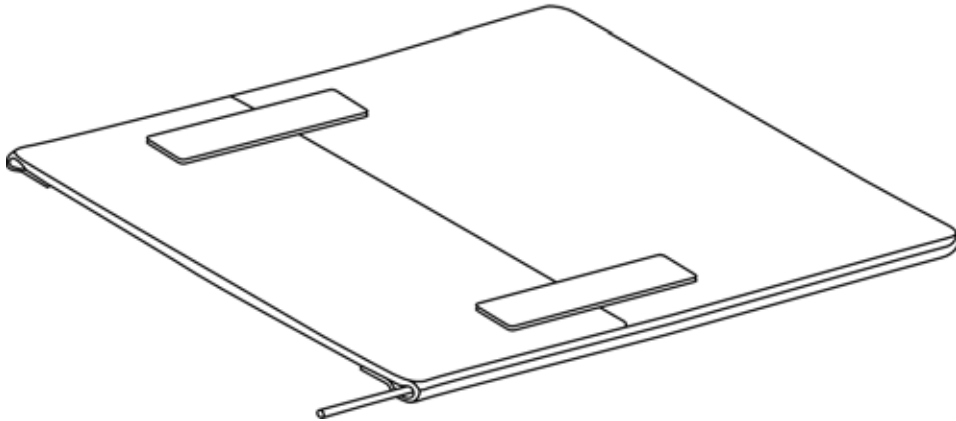


Pos.	Part number	Description	Qty
1	1489052	Adjustable Depth Seat Upholstery W 30,5 cm	1
2	1489013	Adjustable Depth Seat Upholstery W 33 cm	1
3	1489014	Adjustable Depth Seat Upholstery W 35,5 cm	1

Multi Adjustable Seat Upholstery

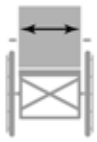		
20,5 / 28 cm	20 / 25 cm	1
20,5 / 28 cm	20 / 30 cm	2
20,5 / 28 cm	30 / 35 cm	3
30,5 / 38 cm	27,5 / 32,5 cm	4
30,5 / 38 cm	32,5 / 37,5 cm	5

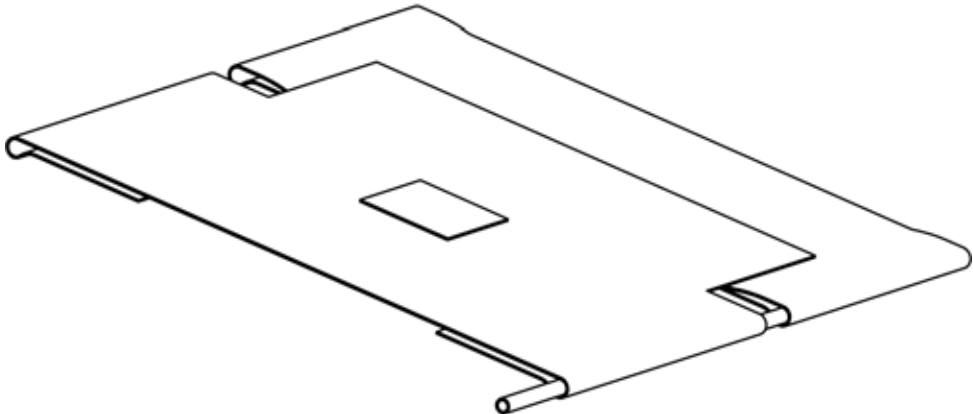


Pos.	Part number	Description	Qty
1	1512012	Adjustable Seat Upholstery W 20,5/28 cm X D 20/25 cm	1
2	1512247	Adjustable Seat Upholstery W 20,5/28 cm X D 25/30 cm	1
3	1512250	Adjustable Seat Upholstery W 20,5/28 cm X D 30/35 cm	1
4	1512248	Adjustable Seat Upholstery W 30,5/38 cm X D 27,5/32,5 cm	1
5	1512249	Adjustable Seat Upholstery W 30,5/38 cm X D 32,5/37,5 cm	1

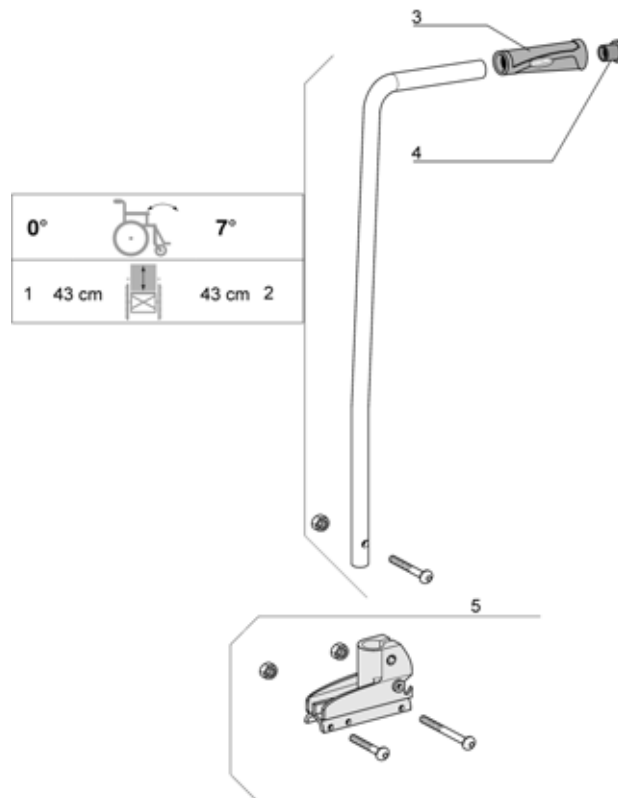Fixed Backrest


Pos.	Part number	Description	Qty
1	1430844	Backrest 0°/43Cm Cane Kit	1
2	1430846	Backrest 7°/43Cm Cane Kit	1
3	1435421	Handle Plastic Black, "Invacare"	1
4	1418112	Crossbrace back cap	1
5	1430848	Fixed Backrest Support Kit	1

Fixed Backrest, Height Adjustable, Push Handle

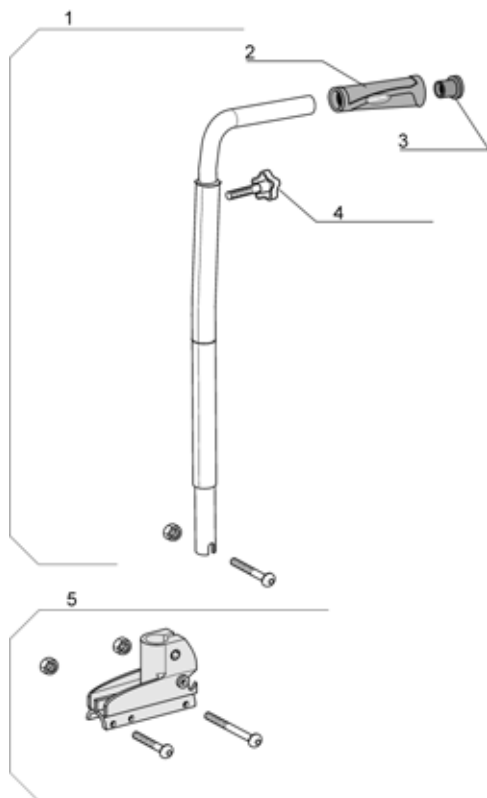


Pos.	Part number	Description	Qty
1	1428641-0601	Adjustable Push Handle Backrest Complete	1
2	1435421	Handle Plastic Black, "Invacare"	1
3	1418112	Crossbrace back cap	1
4	0021-60029	32 mm M6 x 16 Button	1
5	1430848	Fixed Backrest Support Kit	1

Recliner Backrest 0° To 30° Racket

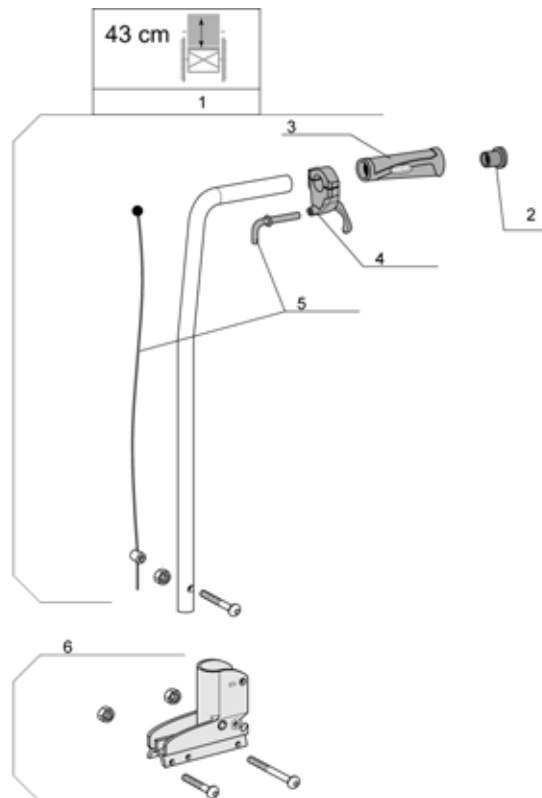


Pos.	Part number	Description	Qty
1	1434491	Reclining Backrest Cane 43 Cm (35,37,39,41 cm) Complete	1
2	1418112	Crossbrace back cap	1
3	1435421	Handle Plastic Black, "Invacare"	1
4	1433747	Recliner Trigger	1
5	1430851	Recliner Cable Kit	1
6	1430850	Recliner Backrest Support Kit	1

Recliner Backrest 0° to 30° Gas Spring

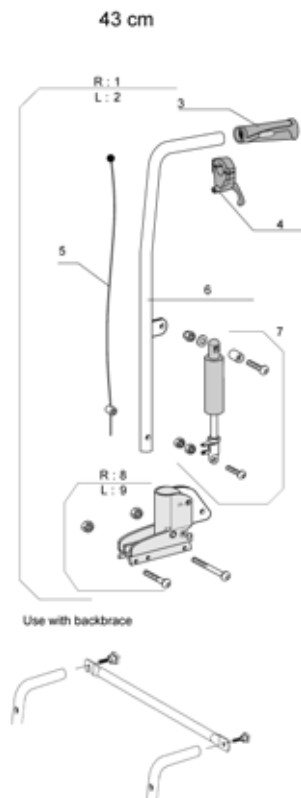


Pos.	Part number	Description	Qty
1	1442538	Reclining Rh Kit Gas Strut Backrest 43 cm	1
2	1442539	Reclining Lh Kit Gas Strut Backrest 43 cm	1
3	1435421	Handle Plastic Black, "Invacare"	1
4	1433747	Recliner Trigger	1
5	1440937	Recliner Gas Cable 43 cm	1
6	1484055-0699	Recliner Gas Tube 43 cm	1
7	1433564	Gas Strut Kit Standard Backrest	1
8	1442507	Rh Recliner Gas Backrest Support Kit	1
9	1442513	Lh Recliner Gas Backrest Support Kit	1

Fold Down Backrest

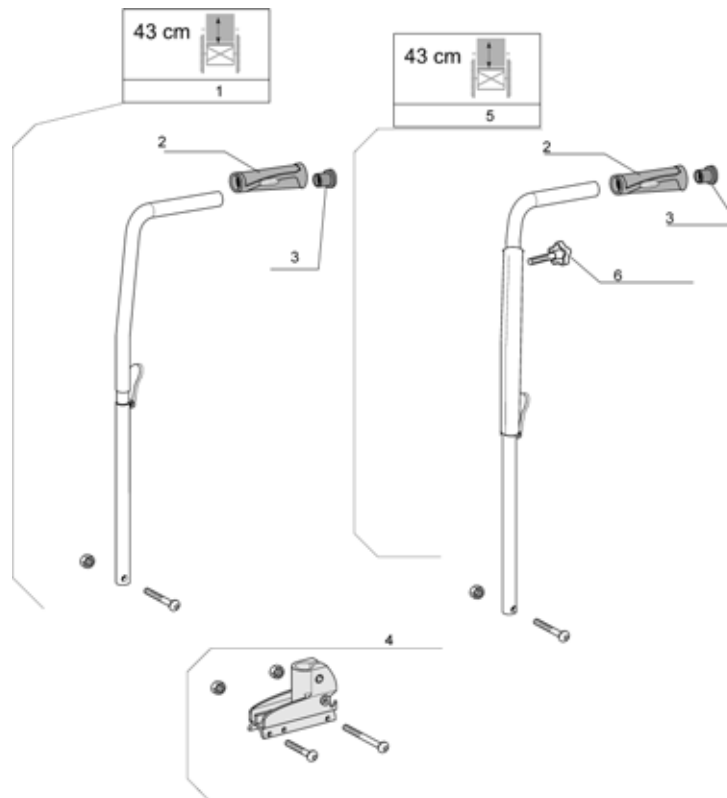


Pos.	Part number	Description	Qty
1	1434492	Folding Backrest Cane 43 cm	1
2	1435421	Handle Plastic Black, "Invacare"	1
3	1418112	Crossbrace back cap	1
4	1430848	Fixed Backrest Support Kit	1
5	1513345	Folding Backrest Adjustable Push Handle 43 cm	1
6	0021-60029	32 mm M6 x 16 Button	1

Adjustable Backrest Upholstery

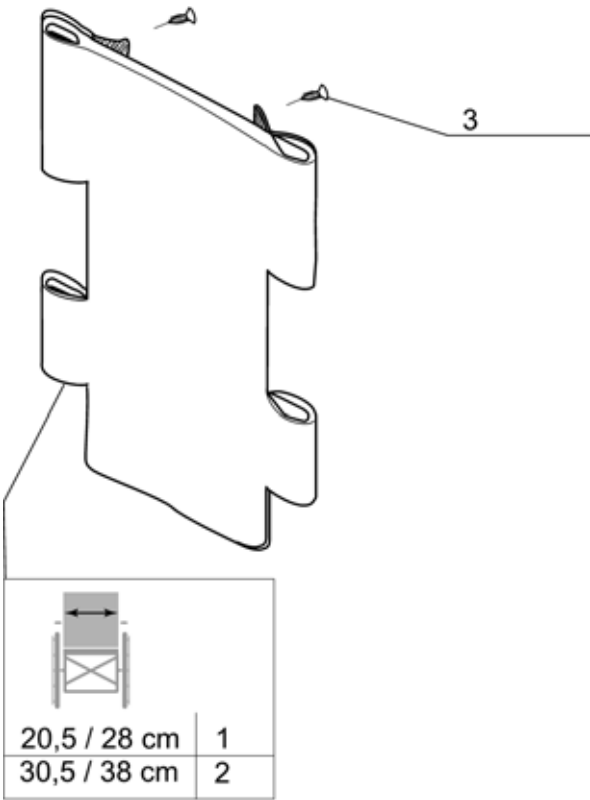


Pos.	Part number	Description	Qty
1	1512251	A3 Junior Adjustable Upholstery W 20,5/28 cm	1
1	1522598	Coloured A3 Junior Adjustable Upholstery W 20,5/28 cm	1
2	1512252	A3 Junior Adjustable Upholstery W 30,5/38 cm	1
2	1522599	Coloured A3 Junior Adjustable Upholstery W 30,5/38 cm	1
3	0029-40005	Nickel-Plated 4.8 x 16 Diameter FBZ Sheet-Metal Screw	1

Tension adjustable backrest upholstery

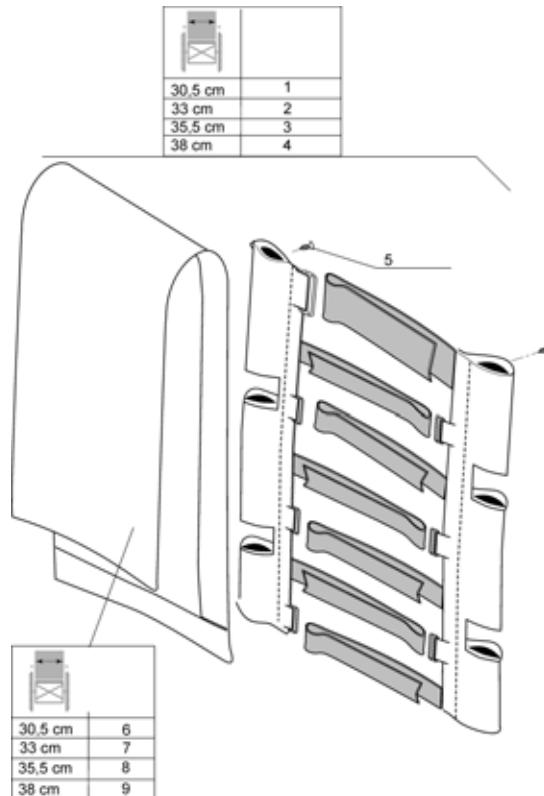


Pos.	Part number	Description	Qty
1	1507192	A3 Junior Adjustable Upholstery W 30,5 cm	1
2	1507193	A3 Junior Adjustable Upholstery W 33 cm	1
3	1507194	A3 Junior Adjustable Upholstery W 35,5 cm	1
4	1513261	A3 Junior Adjustable Upholstery W 38 cm	1
5	0029-40005	Nickel-Plated 4.8 x 16 Diameter FBZ Sheet-Metal Screw	1
6	1506333	Adjustable Back Cover W 30,5 cm Action 3 Junior	1
7	1506334	Adjustable Back Cover W 33 cm Action 3 Junior	1
8	1506335	Adjustable Back Cover W 35,5 cm Action 3 Junior	1
9	1513266	Adjustable Back Cover W 38 cm Action 3 Junior	1

Backrest extension

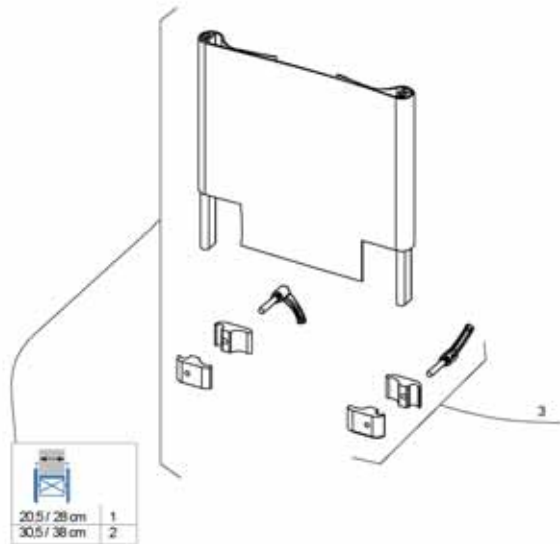


Pos.	Part number	Description	Qty
1	1522603	Backrest extention SW 20,5 to 28 cm	1
2	1522637	Backrest extention SW 30,5 to 38 cm	1
3	1523620	Backrest extension clamps assy	1

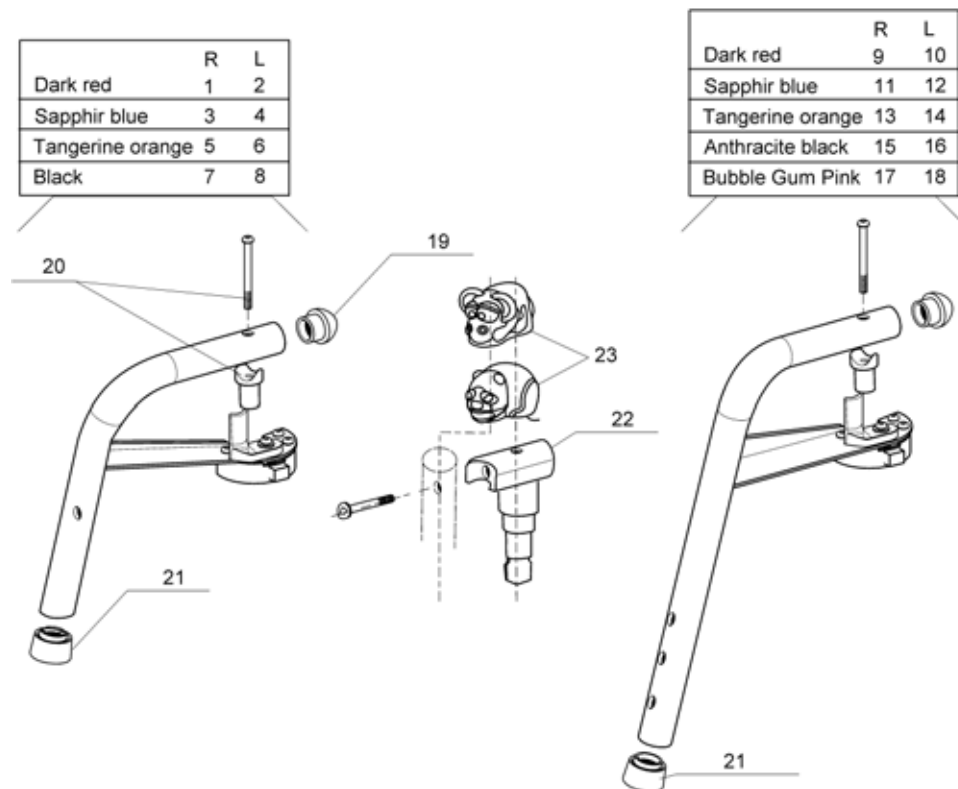Front Hanger 80° & 90°


Pos.	Part number	Description	Qty
1	1428752-0607	Rh Short Hanger A3 Junior Dark Red	1
2	1428753-0607	Lh Short Hanger A3 Junior Dark Red	1
3	1428752-0608	Rh Short Hanger A3 Junior Sapphir Blue	1
4	1428753-0608	Lh Short Hanger A3 Junior Sapphir Blue	1
5	1428752-0613	Rh Short Hanger A3 Junior Tangerine Orange	1
6	1428753-0613	Lh Short Hanger A3 Junior Tangerine Orange	1
7	1428752-0601	Rh Short Hanger A3 Junior Black	1
8	1428753-0601	Lh Short Hanger A3 Junior Black	1
9	1514392-0607	Rh Long Hanger A3 Junior Dark Red	1
10	1514393-0607	Lh Long Hanger A3 Junior Dark Red	1
11	1514392-0608	Rh Long Hanger A3 Junior Sapphir Blue	1
12	1514393-0608	Lh Long Hanger A3 Junior Sapphir Blue	1
13	1514392-0613	Rh Long Hanger A3 Junior Tangerine Orange	1
14	1514393-0613	Lh Long Hanger A3 Junior Tangerine Orange	1
15	1514392-0609	Rh Long Hanger A3 Junior Anthracite Black	1
16	1514393-0609	Lh Long Hanger A3 Junior Anthracite Black	1
17	1514392-0614	Rh Long Hanger A3 Junior Bubble Gum Pink	1
18	1514393-0614	Lh Long Hanger A3 Junior Bubble Gum Pink	1
19	0021-20036	Black 25 mm fine cylindrical cap	1
20	1017148	Foot Rest Support Pivot Kit	1
21	1026124	Foot Rest Support Tip	1
22	1513492	90° Hanger Support	1
23	1513302	90° Hanger Covers Set	1

Elevating Legrest with angle adjustable footrest

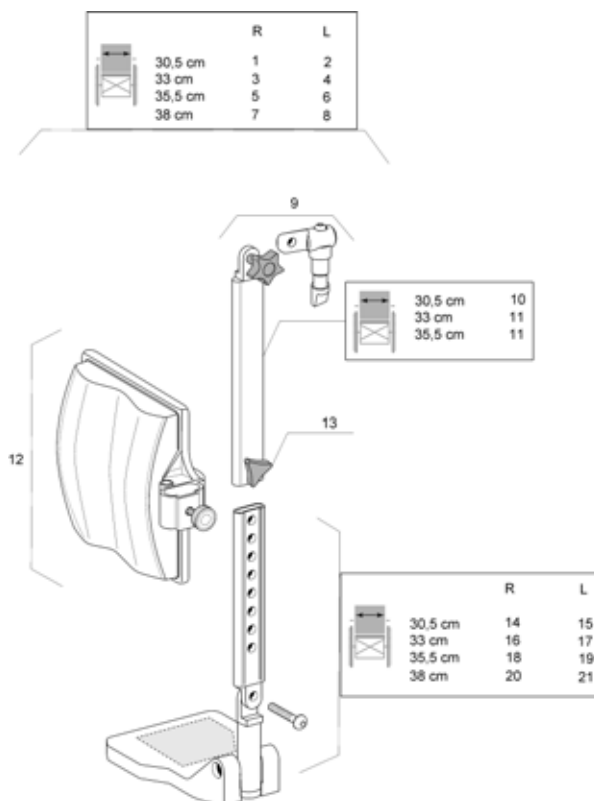


Pos.	Part number	Description	Qty
1	1478439-0601	Rh Complete Legrest 30 cm Without Legrest Cushion	1
2	1449030-0601	Lh Complete Legrest 30 cm Without Legrest Cushion	1
3	1428754-0601	Rh Complete Legrest 33 cm	1
4	1428755-0601	Lh Complete Legrest 33 cm	1
5	1449018-0601	Legrest cpl. RH Junior	1
6	1449019-0601	Legrest cpl. LH Junior	1
7	1512438-0601	Rh Complete Legrest 38 cm	1
8	1512439-0601	Lh Complete Legrest 38 cm	1
9	1428812-0601	Adapter Legrest	1
10	1450950-0601	Sliding Tube Higher Side W 30,5 cm	1
11	1428808-0601	Sliding Tube Higher Side W 33/35,5 cm	1
12	1403981	Complete Legrest cushion width 38 cm	1
13	1434623	Ball Screw Court 7,5mm	1
14	1450909	Rh Articulated Footrest Tube 30 cm	1
15	1450910	Lh Articulated Footrest Tube 30 cm	1
16	1428818-0601	Rh Articulated Footrest Plate Tube 33 cm	1
17	1428823-0601	Lh Articulated Footrest Tube 33 cm	1
18	1449013-0601	Rh Articulated Footrest Tube 35,5 cm	1
19	1449014-0601	Lh Articulated Footrest Tube 35,5 cm	1
20	1513238	Rh Articulated Footrest Tube 38 cm	1
21	1513239	Lh Articulated Footrest Tube 38 cm	1

Elevating Legrest with angle and depth adjustable footrest

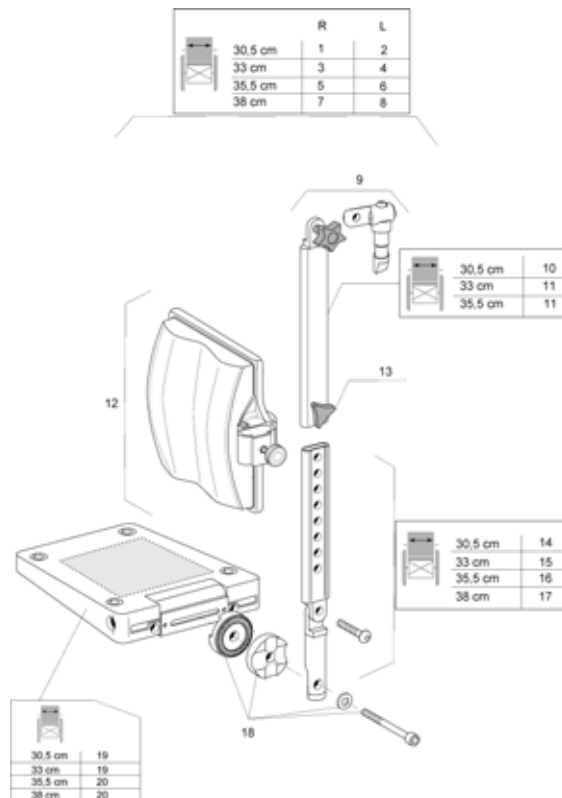


Pos.	Part number	Description	Qty
1	1506482	Rh Complete Legrest 30 cm Without Legrest Cushion + Multi Adjustable Footplate	1
2	1506918	Lh Complete Legrest 30 cm Without Legrest Cushion + Multi Adjustable Footplate	1
3	1506483	Rh Complete Legrest 33 cm + Multi Adjustable Footplate	1
4	1506920	Lh Complete Legrest 33 cm + Multi Adjustable Footplate	1
5	1513297	Rh Complete Legrest 35,5 cm + Multi Adjustable Footplate	1
6	1513298	Lh Complete Legrest 35,5 cm + Multi Adjustable Footplate	1
7	1512527-0601	Rh Complete Legrest 38 cm + Multi Adjustable Footplate	1
8	1512528-0601	Lh Complete Legrest 38 cm + Multi Adjustable Footplate	1
9	1428812-0601	Adapter Legrest	1
10	1450950-0601	Sliding Tube Higher Side W 30,5 cm	1
11	1428808-0601	Sliding Tube Higher Side W 33/35,5 cm	1
12	1403981	Complete Legrest cushion width 38 cm	1
13	1434623	Ball Screw Court 7,5mm	1
14	1507029	Multi Articulated Footrest Tube 30 cm	1
15	1507030	Multi Articulated Footrest Tube 33 cm	1
16	1507031	Multi Articulated Footrest Tube 35,5 cm	1
17	1513240	Multi Articulated Footrest Tube 38 cm	1
18	1430839	SCREW KIT, FOOTREST AA	1
19	5106371	FOOTPLATE 11 AA, CPL.	1
20	5106374	FOOTPLATE 14 AA, CPL.	1

Standard Footplate

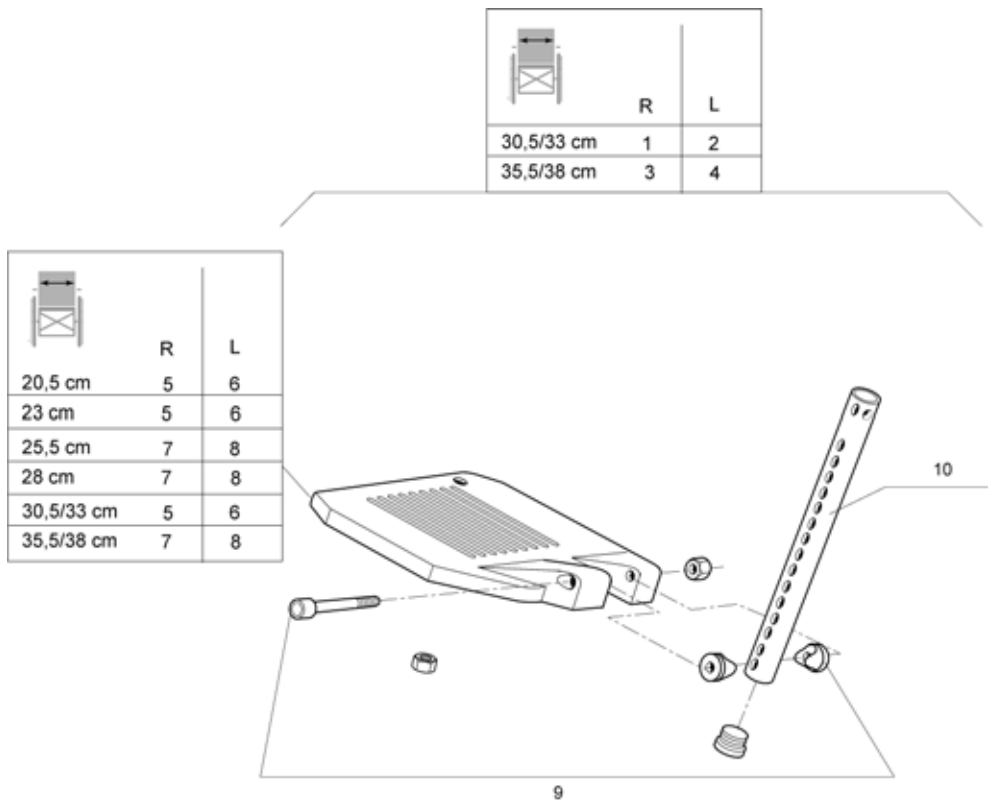


Pos.	Part number	Description	Qty
1	1428758	Rh Fixed Footrest Tube W33 cm	1
2	1428759	Lh Fixed Footrest Tube W33 cm	1
3	1449010	Rh Fixed Footrest Tube W35,5 cm	1
4	1449009	Lh Fixed Footrest Tube W35,5 cm	1
5	0572-04433	RH Footplate 35cm W	1
6	0572-04447	LH Footplate 35cm W	1
7	0572-04434	Lh Footplate 35,5 cm	1
8	0572-04448	Rh Footplate 35,5 cm	1
9	1434612	Kit Fixation Footrest Tube	1
10	1511849	Footrest Tube Junior	1

Adjustable Footplate For 80° Hanger

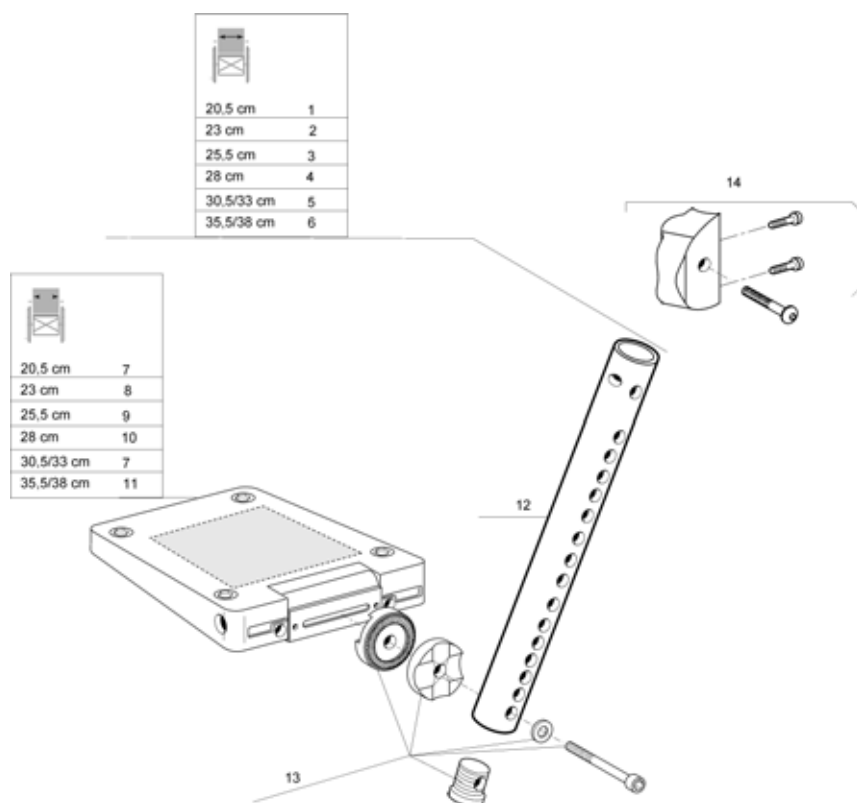


Pos.	Part number	Description	Qty
1	1512255	Adjustable Footrest W 20.5 cm	1
2	1512256	Adjustable Footrest W 23 cm	1
3	1512257	Adjustable Footrest W 25.5/28 cm	1
4	1494144	Adjustable Footrest W 30.5/33	1
5	1513299	Adjustable Footrest W 35.5 cm	1
6	1512259	Adjustable Footrest W 38 cm	1
7	5106373	FOOTPLATE 13 AA, CPL.	1
8	5106375	FOOTPLATE 16 AA, CPL.	1
9	5106376	FOOTPLATE 18 AA, CPL.	1
10	5106371	FOOTPLATE 11 AA, CPL.	1
11	5106374	FOOTPLATE 14 AA, CPL.	1
12	1511849	Footrest Tube Junior	1
13	1430839	SCREW KIT, FOOTREST AA	1
14	1513431	Optimizer Kit (only for 80° & 90° hanger)	1

Adjustable Footplate For 90°Hanger and Stabilizer Frame

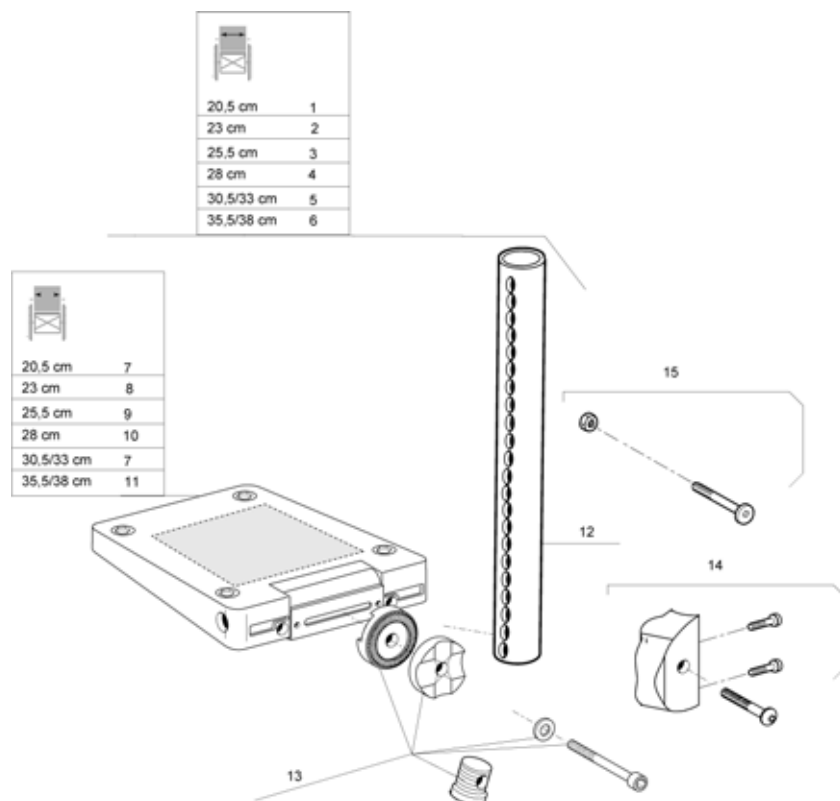


Pos.	Part number	Description	Qty
1	1512287	Adjustable Footrest 90° W 20,5 cm	1
2	1512288	Adjustable Footrest 90° W 23 cm	1
3	1512289	Adjustable Footrest 90° W 25,5/28 cm	1
4	1512291	Adjustable Footrest 90° W 30,5 cm	1
5	1512398	Adjustable Footrest 90° W 35,5 cm	1
6	1512410	Adjustable Footrest 90° W 38 cm	1
7	5106373	FOOTPLATE 13 AA, CPL.	1
8	5106375	FOOTPLATE 16 AA, CPL.	1
9	5106376	FOOTPLATE 18 AA, CPL.	1
10	5106371	FOOTPLATE 11 AA, CPL.	1
11	5106374	FOOTPLATE 14 AA, CPL.	1
12	1506244-0699	Footrest Tube Adjustable Junior 90°	1
13	1430839	SCREW KIT, FOOTREST AA	1
14	1513431	Optimizer Kit (only for 80°& 90° hanger)	1
15	1513432	Stabilizer footrest tube fixation	1

Height Adjustable Armrest N°1

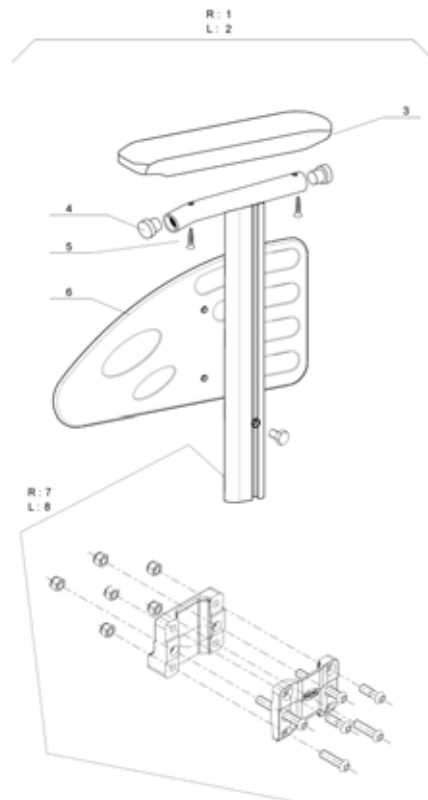


Pos.	Part number	Description	Qty
1	1489055-0601	Rh Armrest Complete	1
2	1489056-0601	Lh Armrest Complete	1
3	1426721	Short Armpad (25 Cm)	1
4	1418112	Crossbrace back cap	1
5	1430422	Armrest 1 Screws Kit	1
6	1425601	Armrest Skirtguard 1	1
7	1484398	A3 Junior Central Support + Rh Tube	1
8	1484399	A3 Junior Central Support + Lh Tube	1

Height Adjustable Armrest N°1 Junior

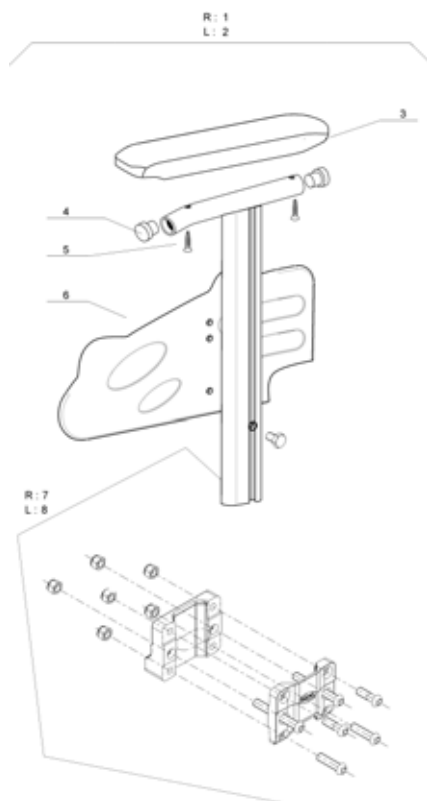


Pos.	Part number	Description	Qty
1	1512645-0601	Rh Armrest Complete Junior	1
2	1512646-0601	Lh Armrest Complete Junior	1
3	1426721	Short Armpad (25 Cm)	1
4	1418112	Crossbrace back cap	1
5	1430422	Armrest 1 Screws Kit	1
6	1510385	Armrest Skirtguard Junior	1
7	1484398	A3 Junior Central Support + Rh Tube	1
8	1484399	A3 Junior Central Support + Lh Tube	1

Height Adjustable Skirtguard Junior

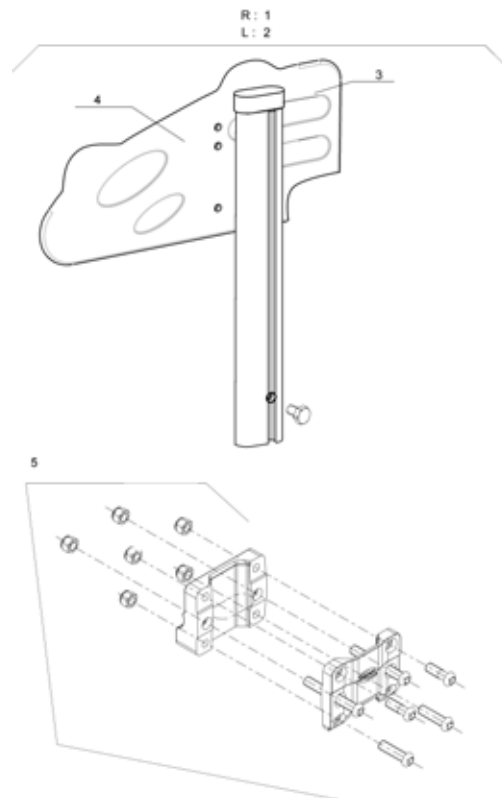


Pos.	Part number	Description	Qty
1	1512647-0601	Rh Skirtguard Junior Complete	1
2	1513214-0601	Lh Skirtguard Junior Complete	1
3	1513515	Skirtguard Junior Cap Kit	1
4	1510385	Armrest Skirtguard Junior	1
5	1450984	Armrest Central Support Assy	1

Height Adjustable Mudguard

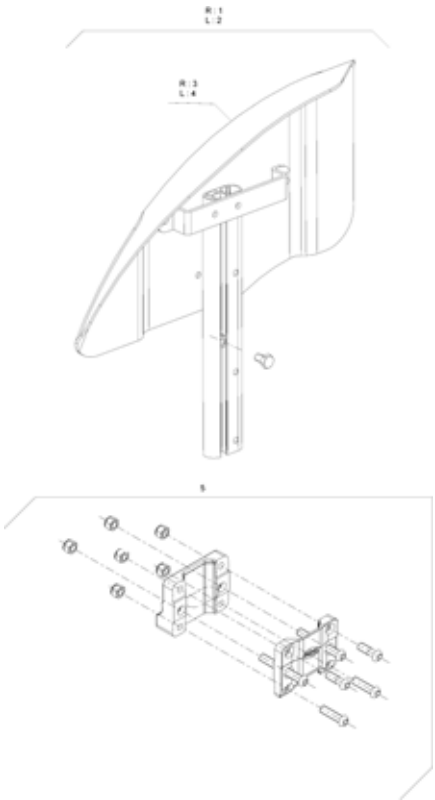


Pos.	Part number	Description	Qty
1	1492738	Rh Mudguard A3-A3J Complete Back Position	1
2	1492739	Lh Mudguard A3-A3J Complete Back Position	1
3	1488805	Rh Mudguard + Fixation	1
4	1488806	Lh Mudguard + Fixation	1
5	1450984	Armrest Central Support Assy	1

Armrest Evolutive <12/2007

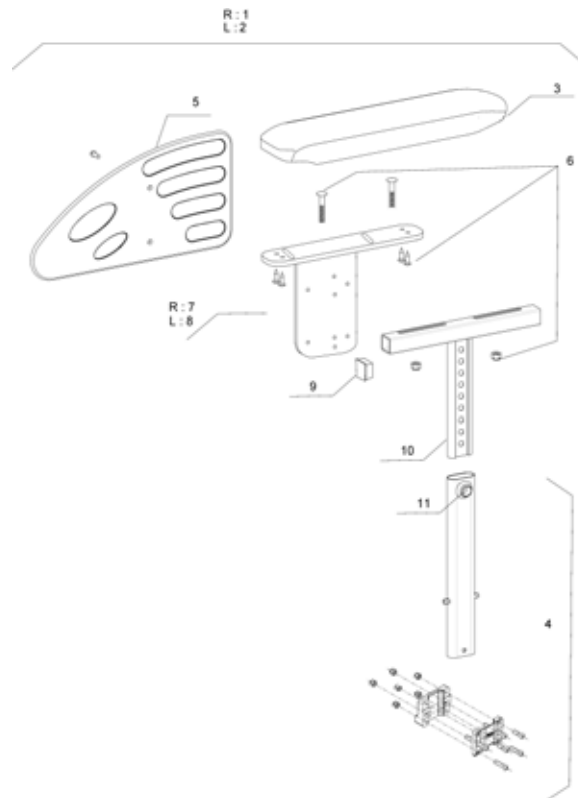


Pos.	Part number	Description	Qty
1	1488954	Rh Armrest A3 Junior Evolutive	1
2	1488955	Lh Armrest A3 Junior Evolutive	1
3	1434676	ARMPAD, SHORT	1
4	1492614	Central Support Armrest Junior Evolutive	1
5	1425601	Armrest Skirtguard 1	1
6	1492613	Screw Kit Armrest Junior Evolutive	1
7	1488835-0601	Rh Armrest Adjustable A3 Evolutive	1
8	1488834-0601	Lh Armrest Adjustable A3 Evolutive	1
9	L03010	20 Mm Square Cap	1
10	1488994-0601	T Support Armrest A3 Evolutive	1
11	5106533	LOCKING SCREW M12X10	1

Fork < 12/2007

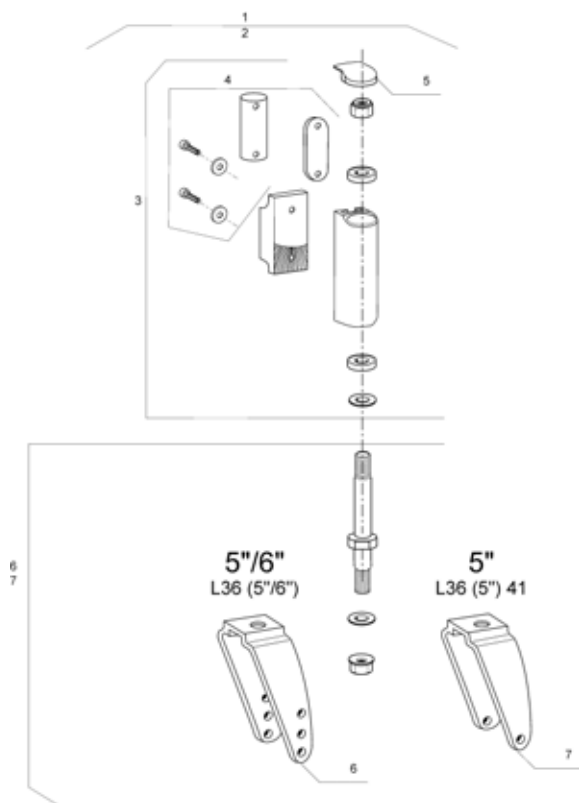


Pos.	Part number	Description	Qty
1	1450026	Complete Fork Kit L36 (6")	1
2	1450027	Complete Fork Kit L36 One Position (5)	1
3	1444804	Fork Support Kit A3-A4	1
4	1450028	Fork Fixation Kit A3-A4	1
5	1449245	Fork Plastic Cap	1
6	1444803	Fork Kit L36 (6")	1
7	1451775	Fork Kit L36 (5")	1

Fork

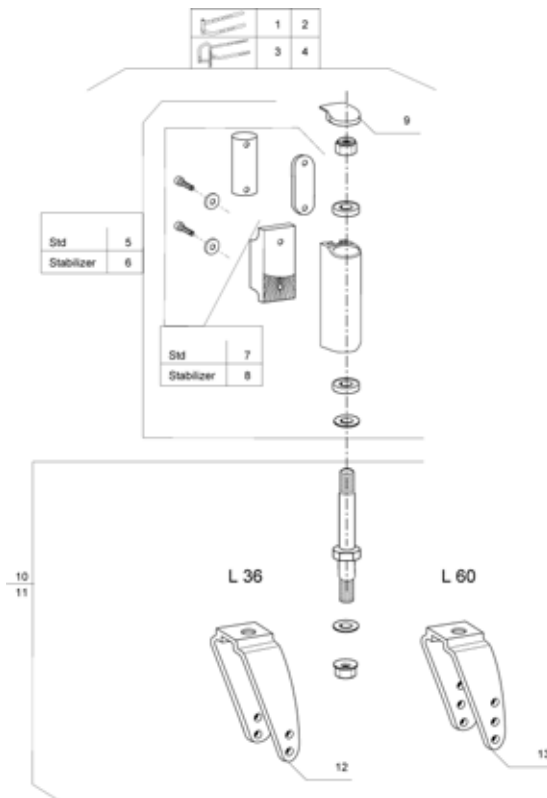


Pos.	Part number	Description	Qty
1	1513247	Complete Fork Kit L36	1
2	1513248	Complete Fork Kit L60	1
3	1513308	Complete Fork Kit L36 Stabilizer	1
4	1513309	Complete Fork Kit L60 Stabilizer	1
5	1444804	Fork Support Kit A3-A4	1
6	1513528	Fork Support Kit Stabilizer	1
7	1450028	Fork Fixation Kit A3-A4	1
8	1513310	Fork Fixation Kit Stabilizer	1
9	1449245	Fork Plastic Cap	1
10	1513536	Fork Complete L36 A3 Junior	1
11	1513547	Fork Complete L60 A3 Junior	1
12	1507323-0601	Fork Kit L36 A3 Junior	1
13	1507324-0601	Fork Kit L60 A3 Junior	1

5" Castors

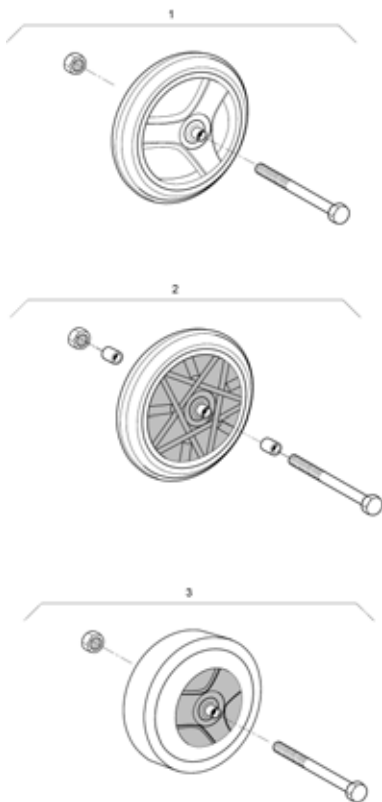


Pos.	Part number	Description	Qty
1	1450173	5"x1" Castor Solid Type + Fixation L 36 mm	1
2	1513250	5"x1" Lighted Castor + Fixation	1
3	1513249	5"x1 3/4" Castor Medium Hard Type + Fixation	1

20" Standard Wheels

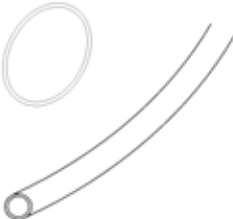20"			
		1" 3/8 	1" 3/8 
		1	2


Pos.	Part number	Description	Qty
1	1445041	20" Pneumatic Basic/19" Alu. Handrim	1
2	1445042	20" Solid Basic/19" Alu. Handrim	1

20" Standard Wheels accessories

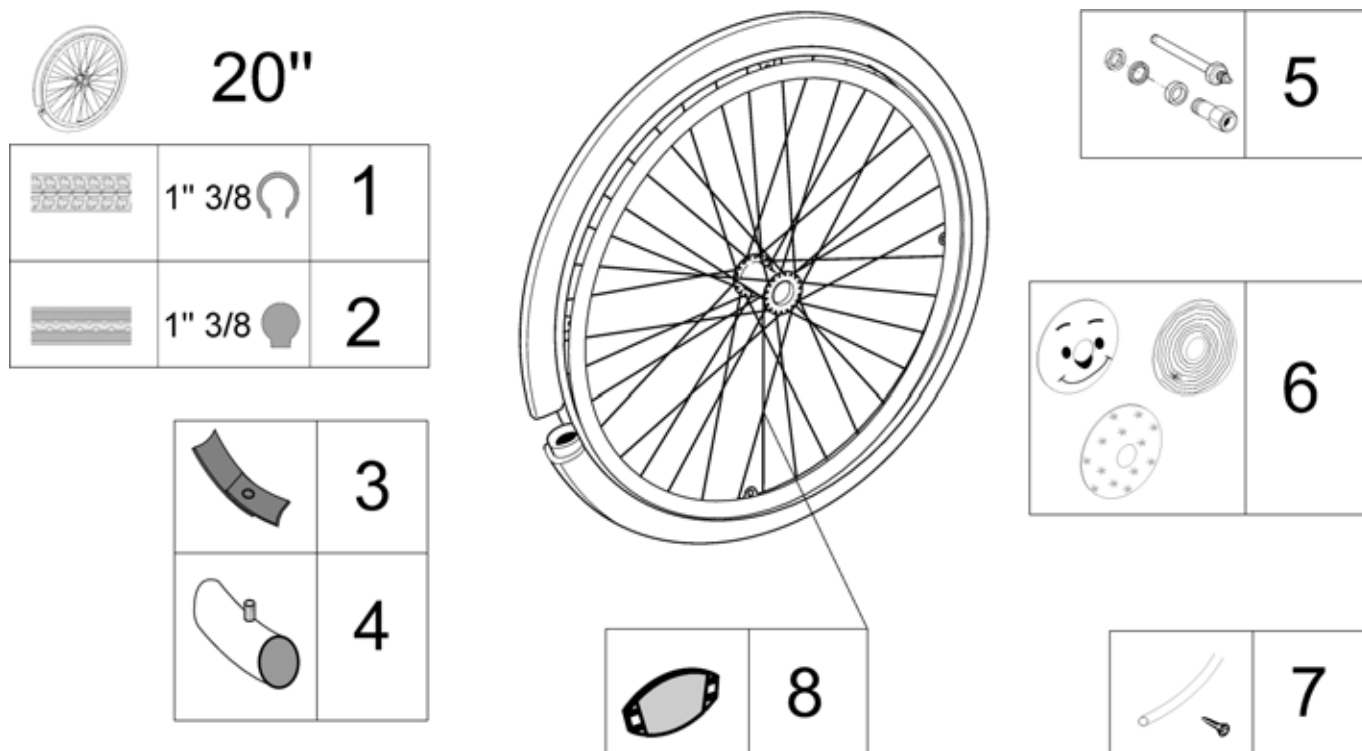

Pos.	Part number	Description	Qty
1	M11009	20 X 1.1.3/8 Pneumatic	1
2	1TYR2002	Grey 20"	1
3	1507198	20" Rim Tape	1
4	0036-20006	20" Inner Tube	1
5	1505288	QR axle for spoked wheels > 03/2007	1
6	1492138	Spoke Guard 20" Smiley	1
6	1513257	Spoke Guard 20" Spider	1
6	1513258	Spoke Guard 20" Flower	1
7	1507241	20" Anodised Handrim	1
8	N90003	Reflectors kit for spoked wheels	1


22" Standard Wheels > 03/2007

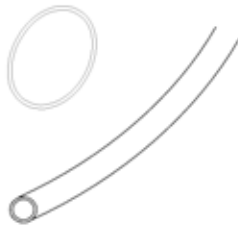22" >03/2007			
		1" 3/8 	1" 3/8 
			
		1	2

Pos.	Part number	Description	Qty
1	1498434	22" Pneumatic Basic/19" Alu. Handrim	1
2	1498432	22"x1.3/8" spoked + anodized handrim + solid tyre > 03/2007	1


Invacare® Action³ Junior | Rear Wheels

22" Standard Wheels < 03/2007

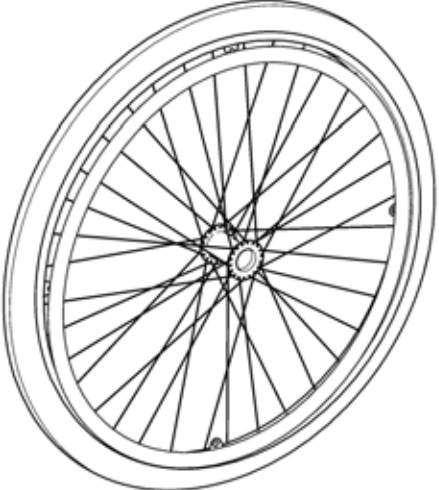22" <03/2007			
		1" 3/8 	1" 3/8 
		1	2

Pos.	Part number	Description	Qty
1	1505306	Pair 22"x1.3/8" spoked + anodized handrim + pneumatic tyre + QR axle < 03/2007	1
2	1505308	Pair 22"x1.3/8" spoked + anodized handrim + solid tyre + QR axle < 03/2007	1





22" Standard Wheels accessories

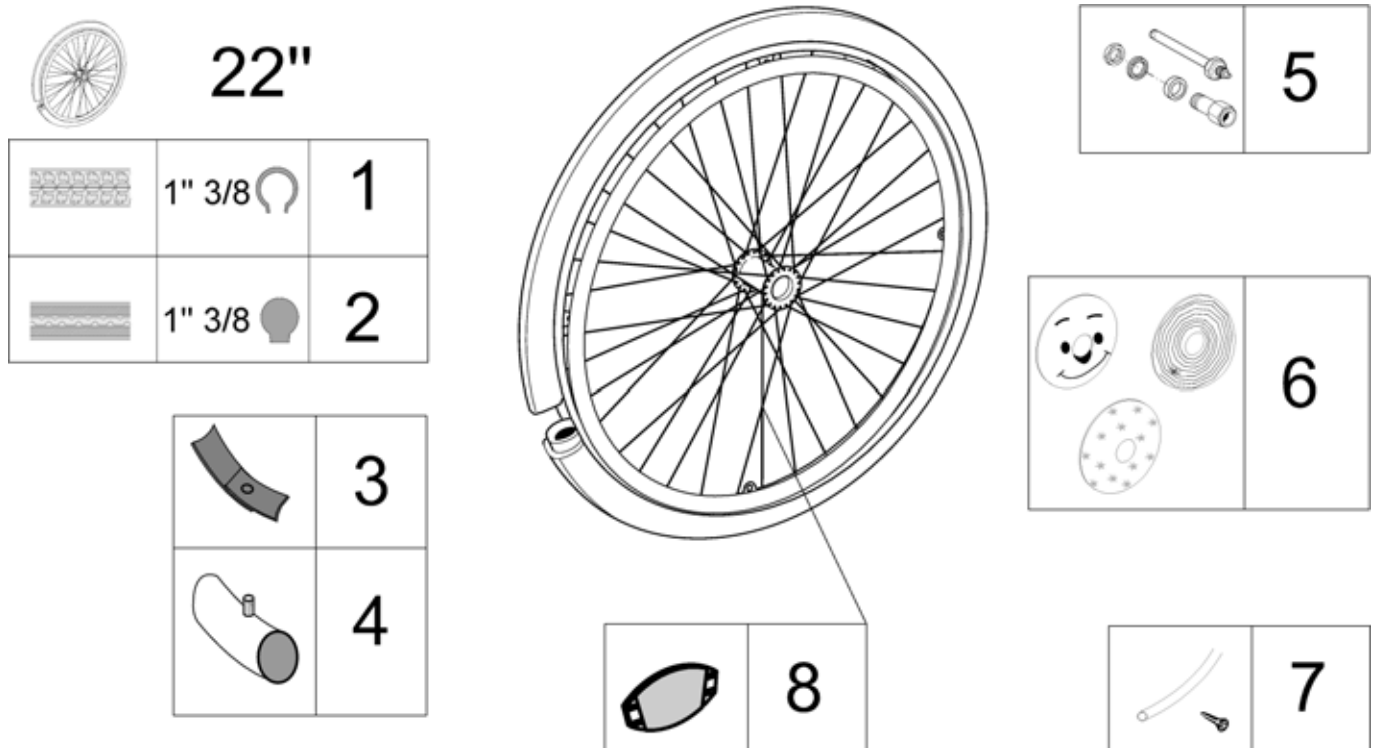






Pos.	Part number	Description	Qty
5	1431276	QR axle for spoked wheels < 03/2007	1
1	0036-30035	22"x1.3/8" pneumatic tyre	1
2	0036-10014	22"x1.3/8" solid tyre	1
3	0036-40002	22" rim tape	1
4	0036-20007	22"x1.3/8" inner tube	1
5	1505288	QR axle for spoked wheels > 03/2007	1
6	1522862	Spoke Guard 22" Smiley	1
6	1522863	Spoke Guard 22" Spider	1
6	1522864	Spoke Guard 22" Flower	1
7	1431320	22" anodized handrim 4 fixations < 03/2007	1
8	N90003	Reflectors kit for spoked wheels	1

22"Hub brake wheels with accessories > 03/2007

22"

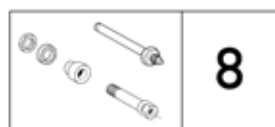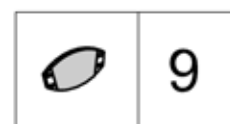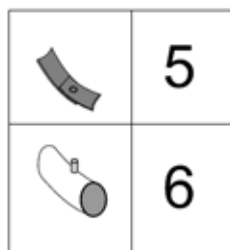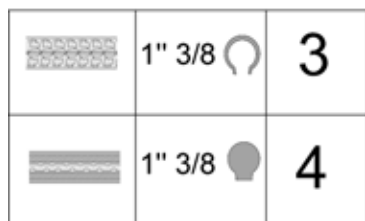


Pos.	Part number	Description	Qty
1	1498466	22"x1.3/8" hub brake spoked + anodized handrim + pneumatic tyre > 06/2007	1
2	1498461	22"x1.3/8" hub brake spoked + anodized handrim + solid tyre > 06/2007	1
3	0036-30035	22"x1.3/8" pneumatic tyre	1
4	0036-10014	22"x1.3/8" solid tyre	1
5	0036-40002	22" rim tape	1
6	0036-20007	22"x1.3/8" inner tube	1
7	1522862	Spoke Guard 22" Smiley	1
7	1522863	Spoke Guard 22" Spider	1
7	1522864	Spoke Guard 22" Flower	1
8	1505290	QR axle for hub brake spoked wheels	1
9	N90003	Reflectors kit for spoked wheels	1
10	1505299	22" anodized handrim 4 fixations > 03/2007	1

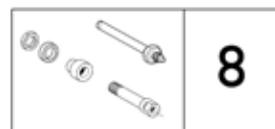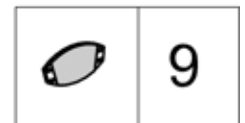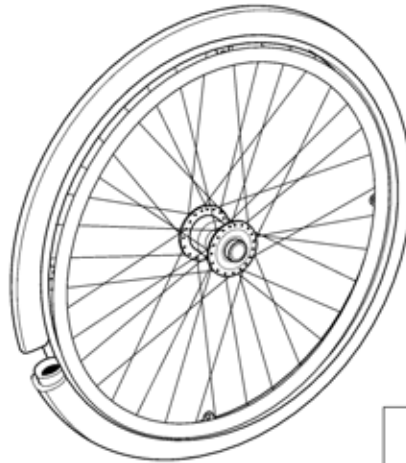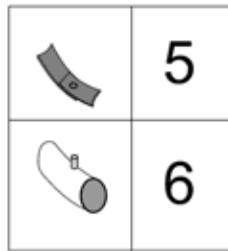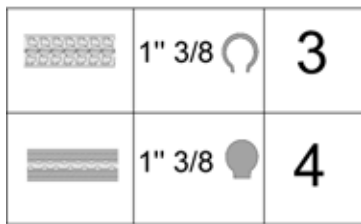22"Hub brake wheels with accessories < 03/2007**22"**


Pos.	Part number	Description	Qty
1	1505306	Pair 22"x1.3/8" spoked + anodized handrim + pneumatic tyre + QR axle < 03/2007	1
2	1505308	Pair 22"x1.3/8" spoked + anodized handrim + solid tyre + QR axle < 03/2007	1
3	0036-30035	22"x1.3/8" pneumatic tyre	1
4	0036-10014	22"x1.3/8" solid tyre	1
5	0036-40002	22" rim tape	1
6	0036-20007	22"x1.3/8" inner tube	1
7	1522862	Spoke Guard 22" Smiley	1
7	1522863	Spoke Guard 22" Spider	1
7	1522864	Spoke Guard 22" Flower	1
8	1431278	Hub Brake Quick Release Axle	1
9	N90003	Reflectors kit for spoked wheels	1
10	1431320	22" anodized handrim 4 fixations < 03/2007	1

Pump And Support

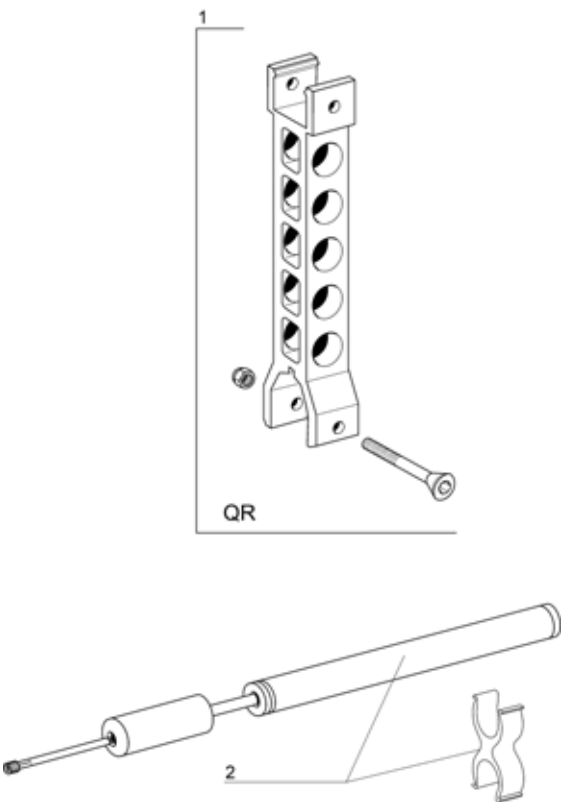


Pos.	Part number	Description	Qty
1	1431282	Rear wheel support for QR axle	1
2	1419910	Pump and its support	1

Push Brakes : Standard And With Extension

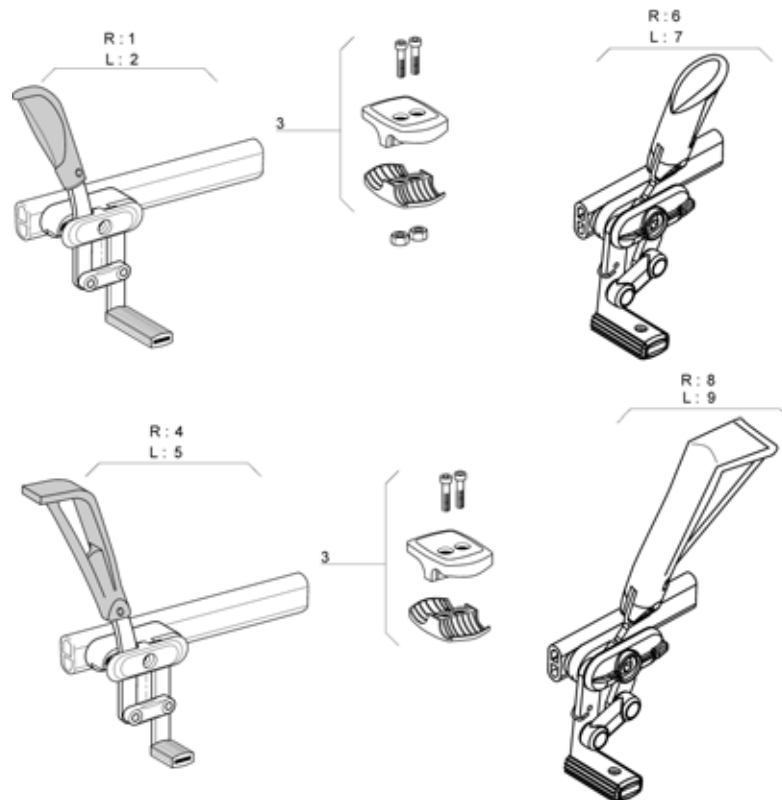


Pos.	Part number	Description	Qty
1	1425635	Brake Rh Long	1
2	1425634	Brake Lh Long	1
3	1423107	Complete Brake Composite Clamp	1
4	1425603	Extended Brake Rh Long	1
5	1425602	Extended Brake Lh Long	1
6	1425643	Rh Reverse Push to Lock Brake A3	1
7	1425642	Lh Reverse Push to Lock Brake A3	1
8	1425641	Rh Reverse Complete Brake With Lever Extension	1
9	1425640	Lh Reverse Complete Brake with lever extension	1

Attendant Hub Brake

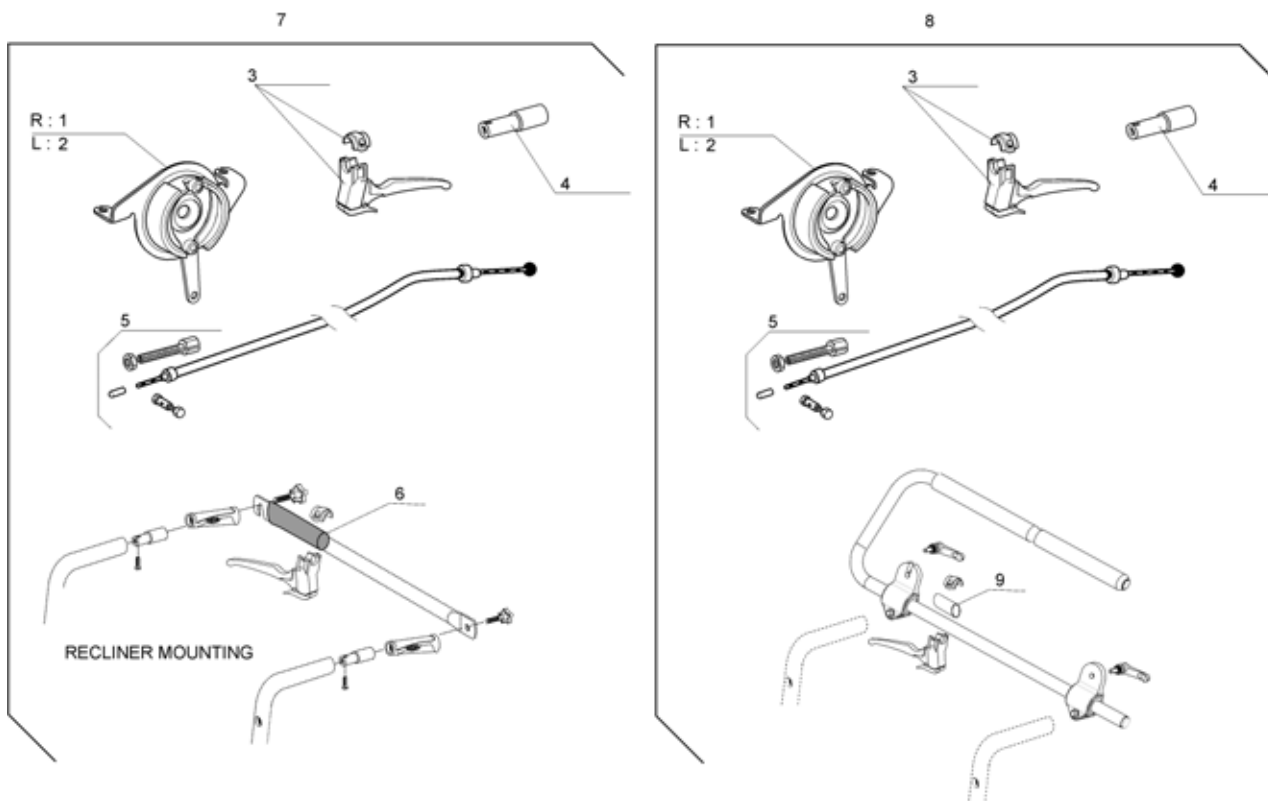


Pos.	Part number	Description	Qty
1	1423292	Rh Hub Brake Plate 70mm	1
2	1423293	Lh Hub Brake Plate 70mm	1
3	1420527	Black Brake Lever dia 22mm	1
4	1430412	Back brace insert for fixed backrest tube	1
5	1430204	Attendant Hub Brakes Complete Mechanism (pair)	1
6	1434626	Backbrace Handle	1
7	1524075	Attendant Hub Brak Kit (W/O Wheel \$ Axle) With Back Brace	1
8	1524076	Attendant Hub Brak Kit (W/O Wheel \$ Axle) With Push Bar	1
9	1511842	Push Bar Adapter Brake Handle	1

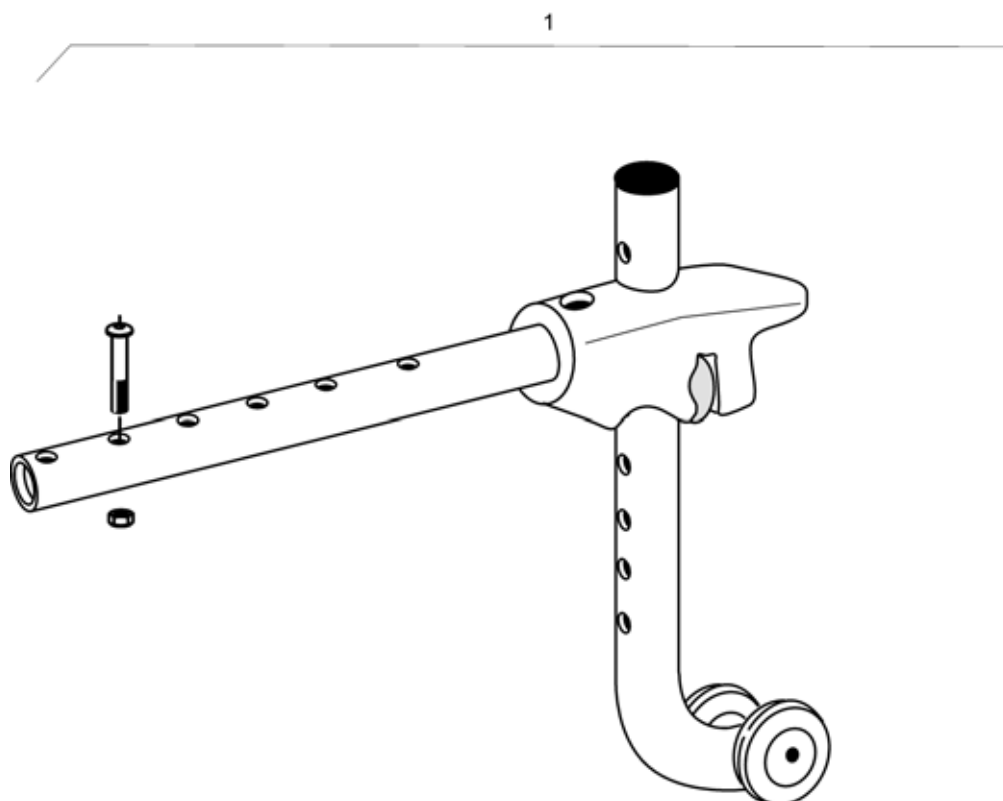
Rear Anti Tippers

Pos.	Part number	Description	Qty
1	1484890	Anti Tip Device Complet A3/A4	1

Tipping Rear Anti Tipper

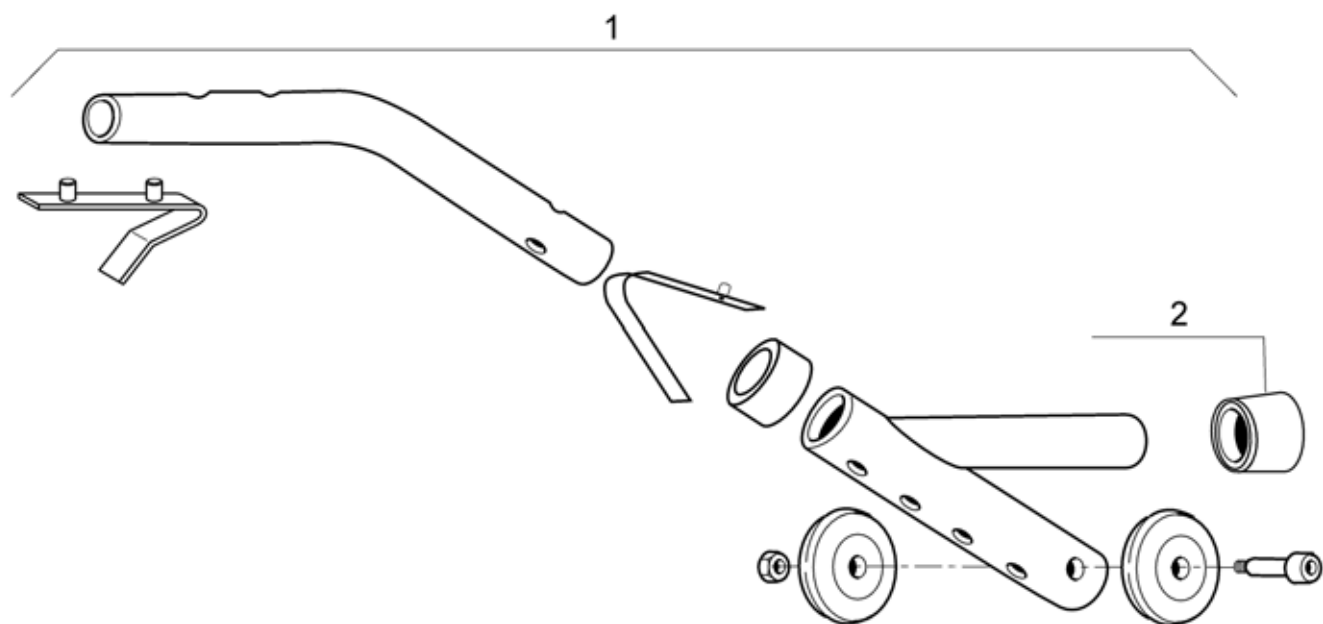


Pos.	Part number	Description	Qty
1	1513251	Tipping & Rear Anti Tipper Tube A3 Junior	1
2	0021N05386	1" cap	1

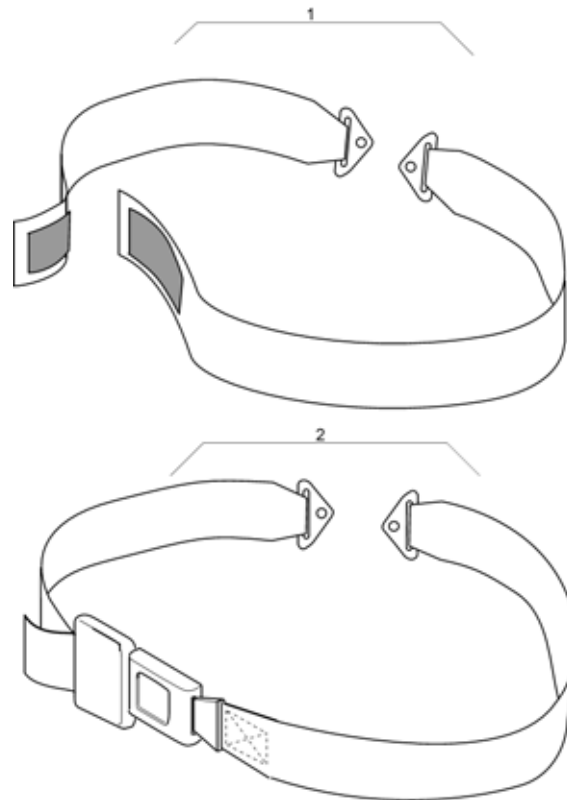
Belts

Pos.	Part number	Description	Qty
1	1492141	Velcro Safety Belt A3 Junior	1
2	1492140	Buckle Safety Belt A3 Junior	1

Transfer Kit

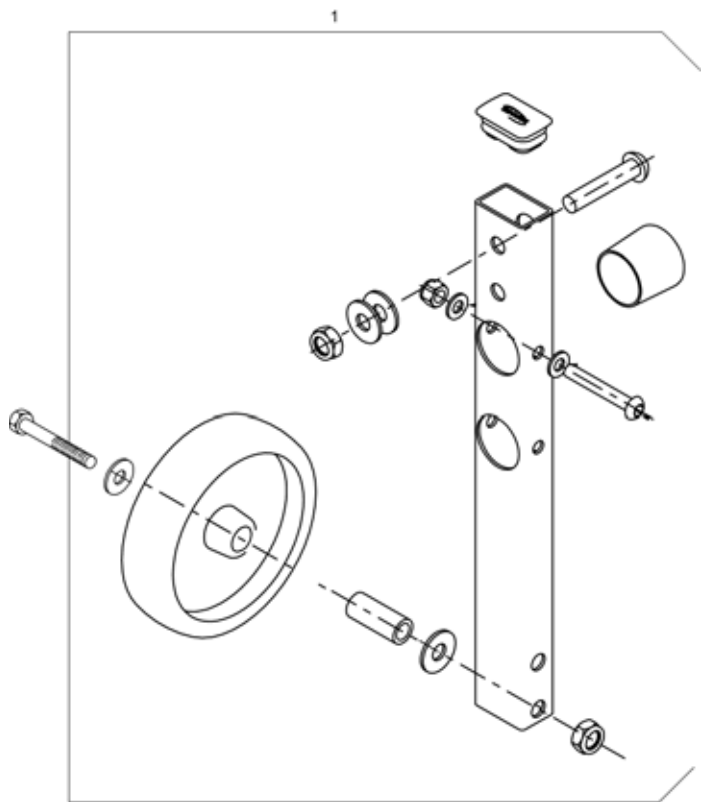



Pos.	Part number	Description	Qty
1	1492139	Transfert Kit Complete	1

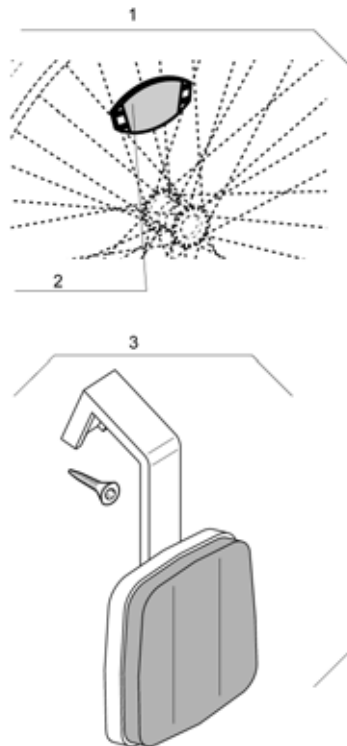Reflector Kit

Pos.	Part number	Description	Qty
1	1430293	Spoked wheel & Frame Reflectors Kit	1
2	N90003	Reflectors kit for spoked wheels	1
3	1430184	Frame Reflectors Kit	1

Cushion

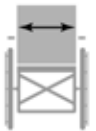
	30 cm	
30,5 cm	1	-
33 cm	-	2
35,5 cm	-	2

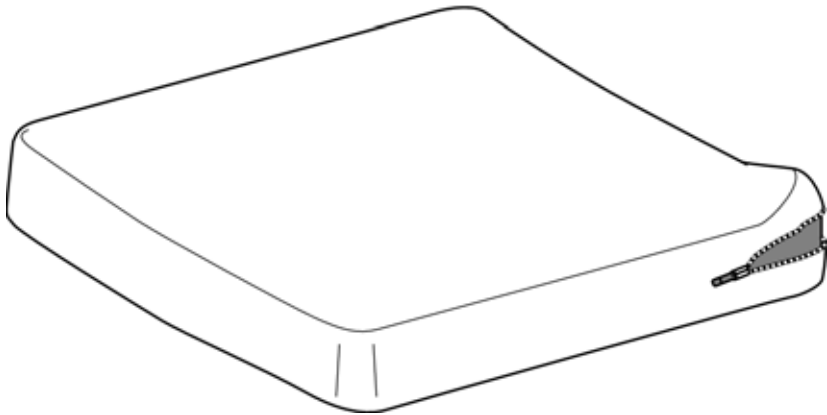

Pos.	Part number	Description	Qty
1	1487818	Standard Foam Cushion W 29Cm X D 29 cm	1
2	1487819	Standard Foam Cushion W 34Cm X D 34 cm	1


Essential Junior Cushion

		
20,5 cm	20 / 25 cm	1
20,5 cm	25 / 30 cm	2
23 cm	20 / 25 cm	3
23 cm	25 / 30 cm	4
25,5 cm	20 / 25 cm	5
25,5 cm	25 / 30 cm	6
28 cm	20 / 25 cm	7
28 cm	25 / 30 cm	8
30,5 cm	27,5 / 32,5 cm	9
30,5 cm	32,5 / 37,5 cm	10
33 cm	27,5 / 32,5 cm	11
33 cm	32,5 / 37,5 cm	12
35,5 cm	27,5 / 32,5 cm	13
35,5 cm	32,5 / 37,5 cm	14
38 cm	27,5 / 32,5 cm	15
38 cm	32,5 / 37,5 cm	16

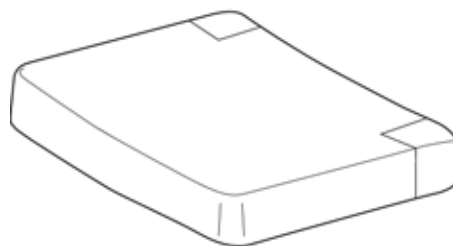


Pos.	Part number	Description	Qty
1	1500963	Essential Junior cushion A W 20,5 / D 20 X 25 cm	1
2	1500964	Essential Junior cushion A W 20,5 / D 25 X 30 cm	1
3	1500965	Essential Junior cushion A W 23 / D 20 X 25 cm	1
4	1500966	Essential Junior cushion A W 23 / D 25 X 30 cm	1
5	1500967	Essential Junior cushion A W 25,5 / D 20 X 25 cm	1
6	1500968	Essential Junior cushion A W 25,5 / D 25 X 30 cm	1
7	1500969	Essential Junior cushion A W 28 / D 20 X 25 cm	1
8	1500970	Essential Junior cushion A W 28 / D 25 X 30 cm	1
9	1500971	Essential Junior cushion A W 30,5 / D 27,5 X 32,5 cm	1
10	1500972	Essential Junior cushion A W 30,5 / D 32,5 X 37,5 cm	1
11	1500973	Essential Junior cushion A W 33 / D 27,5 X 32,5 cm	1
12	1500974	Essential Junior cushion A W 33 / D 32,5 X 37,5 cm	1
13	1500975	Essential Junior cushion A W 35,5 / D 27,5 X 32,5 cm	1
14	1500976	Essential Junior cushion A W 35,5 / D 32,5 X 37,5 cm	1
15	1500977	Essential Junior cushion A W 38 / D 27,5 X 32,5 cm	1
16	1500978	Essential Junior cushion A W 38 / D 32,5 X 37,5 cm	1

Headrest & Neckrest

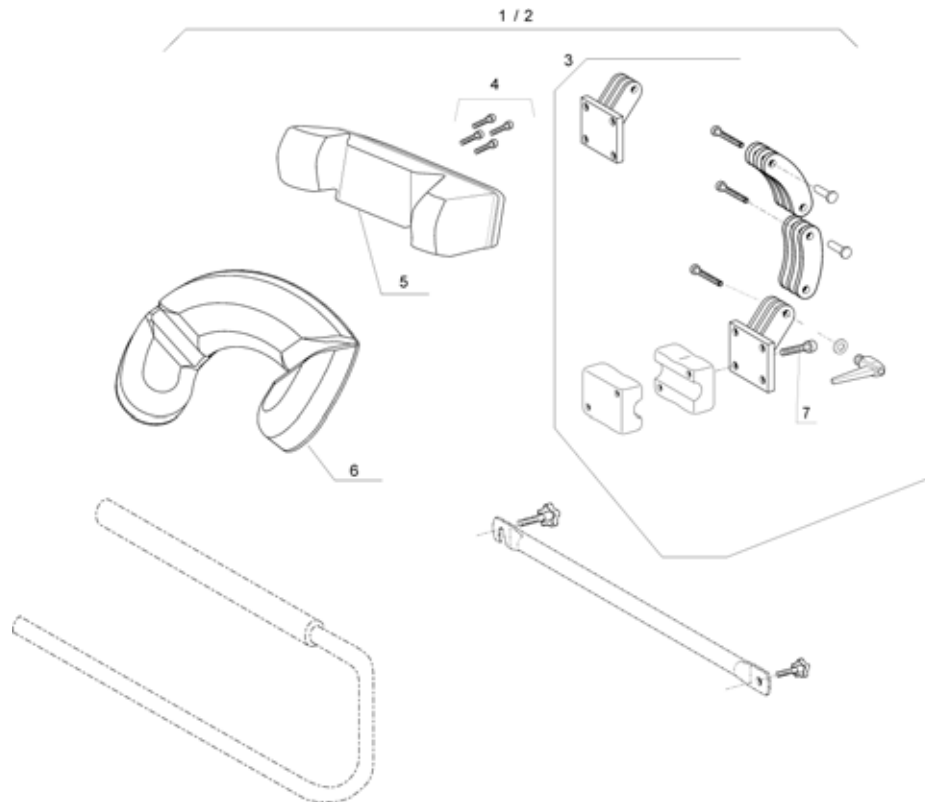


Pos.	Part number	Description	Qty
1	1492010	Neckrest A3 Junior Complete	1
2	1513245	Headrest A3 Junior Complete	1
3	1478706	Neckrest A3 Junior Articulation Complete	1
4	0029-40032	6.8 Zinc CHC M5 x 20 Screw	1
5	5108002	NECKREST CUSH. STD., INCL. UPHOLSTERY, BLACK	1
6	5108473	NECKR CUSH. ROUND, INCL. UPHOLST., BLACK	1
7	0029-40200	8.8 Nickel-Plated M5 x 35 Fbz Screw	1

Heel Strap - Calf Strap Junior

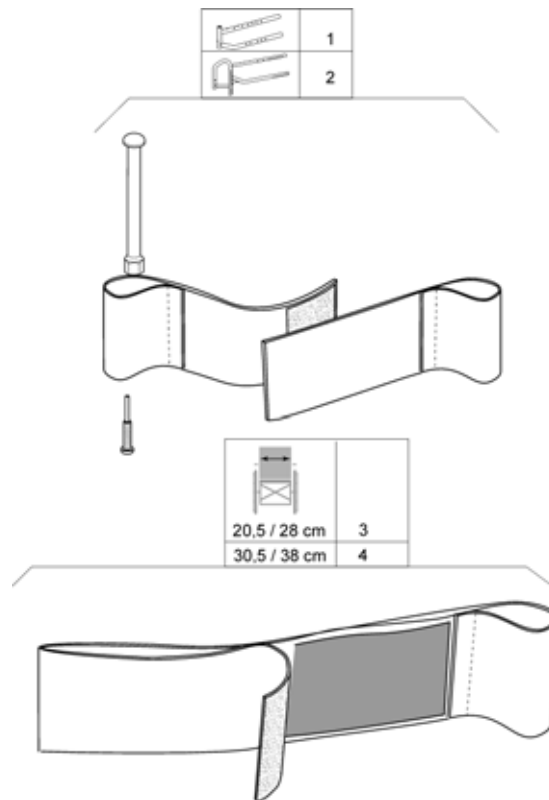


Pos.	Part number	Description	Qty
1	1478467	Adjustable Heel Strap	1
2	1513300	Adjustable Heel Strap Stabilizer	1
3	1513301	Adjustable Calf Strap Small (W20,5to 28cm)	1
4	0572-05794	Adjustable Calf Strap 38/41cm	1

Back Brace & Push Bar

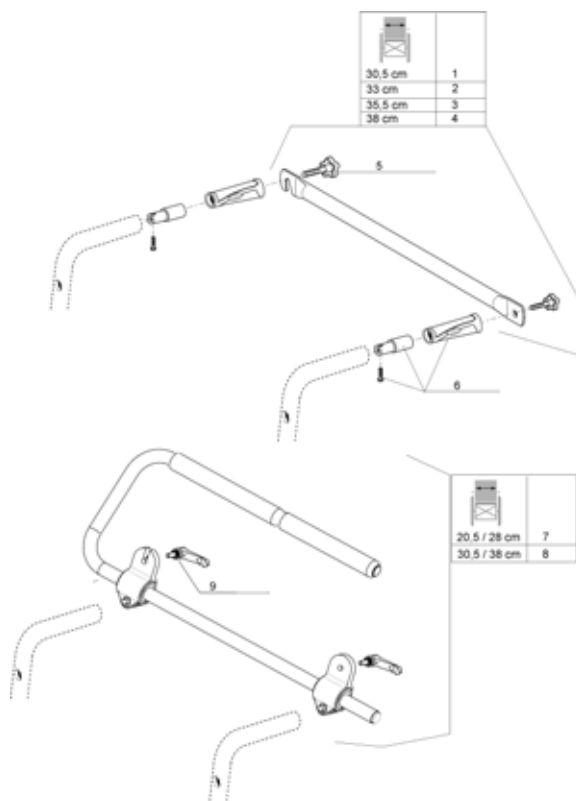


Pos.	Part number	Description	Qty
1	1492007	Back Brace Kit Assy W 30,5 cm	1
2	1492008	Back Brace Kit Assy W 33 cm	1
3	1492009	Back Brace Kit Assy W 35,5 cm	1
4	1428549	Back Brace Kit Assy W 38 cm	1
5	0021-60029	32 mm M6 x 16 Button	1
6	1430412	Back brace insert for fixed backrest tube	1
7	1513242	Push Bar Kit Assy W 20,5/28 cm:	1
8	1513243	Push Bar Kit Assy W 30,5/38 cm:	1
9	1514326	Indexable Lever M6 X 30	1

1

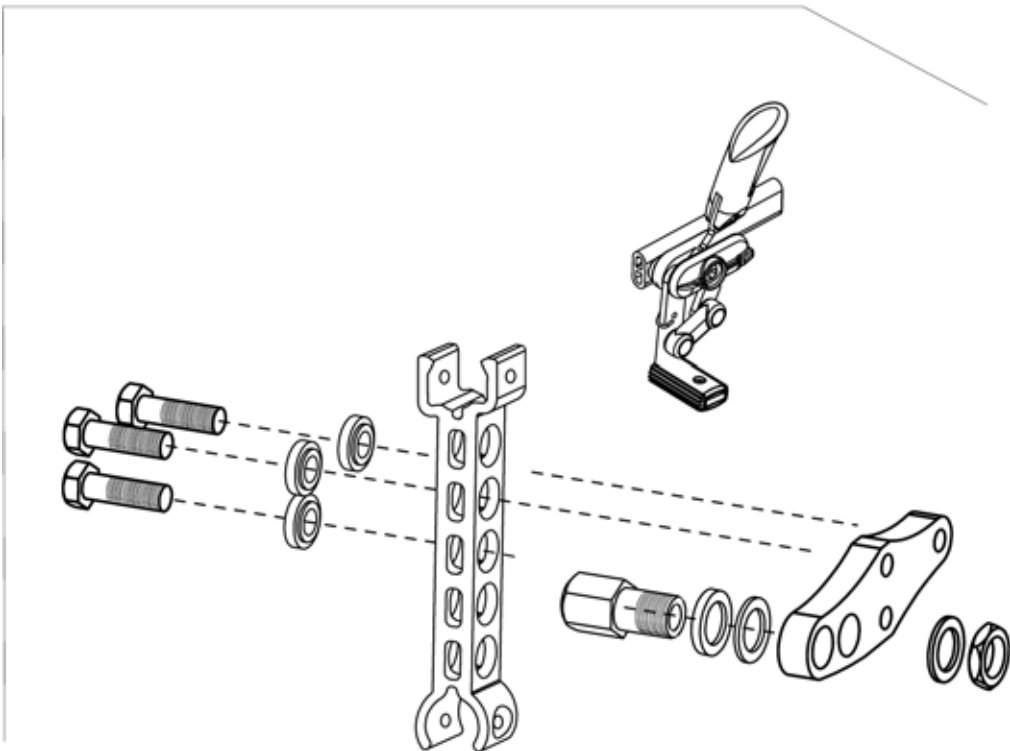


Pos.	Part number	Description	Qty
1	AP1468002	Dual Amputee Position Kit	1

Transit Rear Wheel Pusch Brake > 12/2007

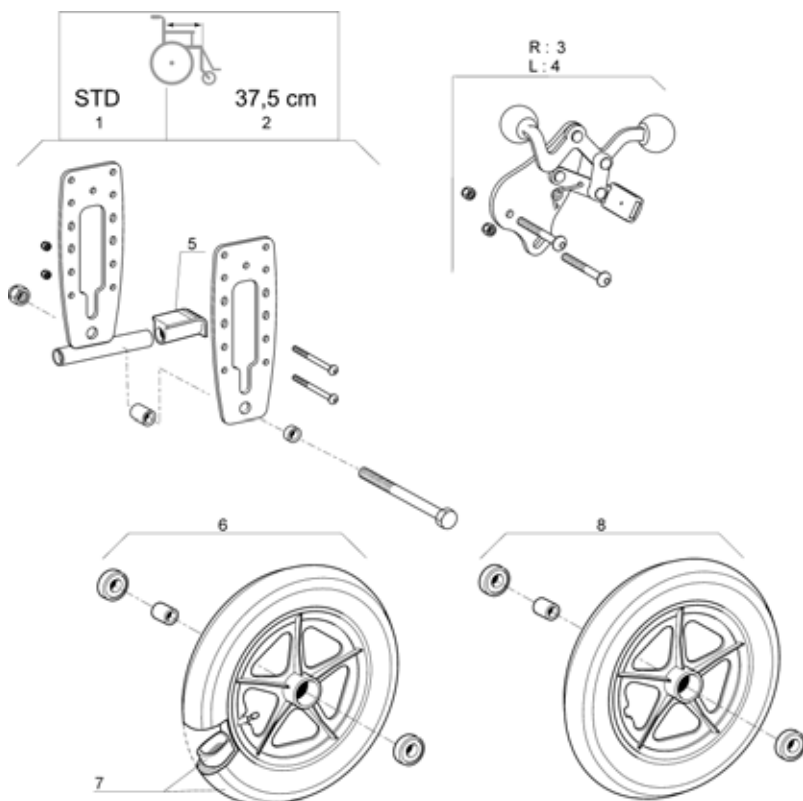


Pos.	Part number	Description	Qty
1	1513637	Action 3 Standard Transit Rear Support Complete	1
2	1513638	Action 3 Standard Transit Rear Support D 37,5 cm Complete	1
3	1513639	Action 3 Junior Rh Transit Push Brake	1
4	1513640	Action 3 Junior Lh Transit Push Brake	1
5	0021-20011	Black Foot Support Cap	1
6	6001789	Transit Complete Pneumatic Tyre 12.1/2" wheel	1
7	1431313	12" transit Pneumatic Tyre + Inner tube	1
8	6001790	Transit Complete Solid Tyre 12.1/2" Wheel	1

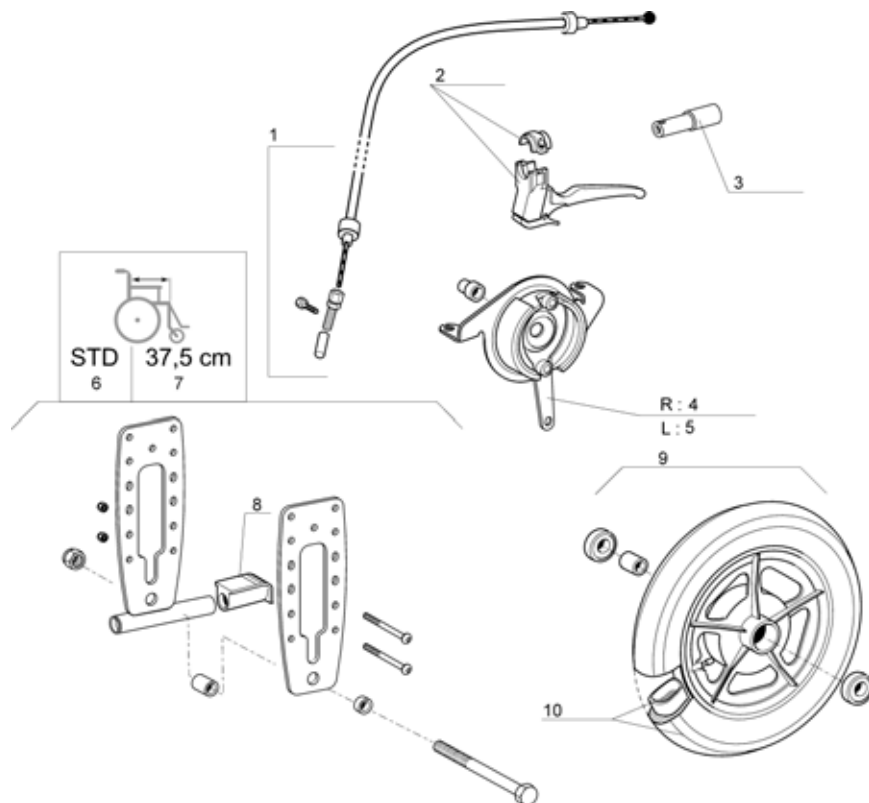Transit Rear Wheel Hub Brake >12/2007


Pos.	Part number	Description	Qty
1	1431285	Attendant Hub Brakes Complete Mechanism (Pair)	1
2	1420527	Black Brake Lever dia 22mm	1
3	1430412	Back brace insert for fixed backrest tube	1
4	1423292	Rh Hub Brake Plate 70mm	1
5	1423293	Lh Hub Brake Plate 70mm	1
6	1513641	Action 3 Hub Brake Transit Rear Support Complete	1
7	1513642	Action 3 Hub Brake Transit Rear Support 37,5 cm Complete	1
8	0021-20011	Black Foot Support Cap	1
9	0572-03002	Complete 70 Hub Brake Composite 12.1/2" Wheel	1
10	1431313	12" transit Pneumatic Tyre + Inner tube	1

Spoke Guard

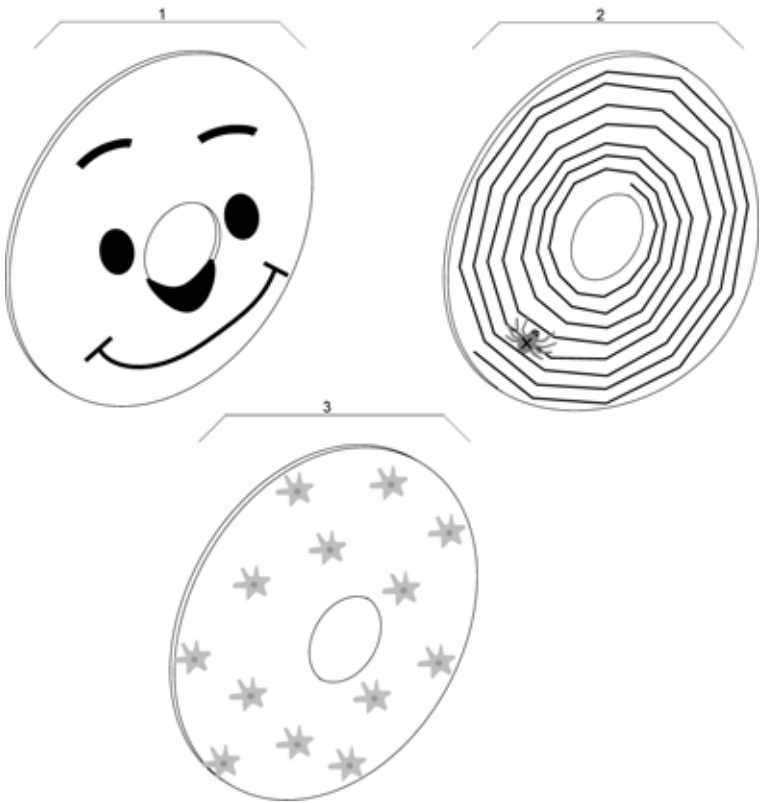


Pos.	Part number	Description	Qty
1	1492138	Spoke Guard 20" Smiley	1
1	1522862	Spoke Guard 22" Smiley	1
2	1513257	Spoke Guard 20" Spider	1
2	1522863	Spoke Guard 22" Spider	1
3	1513258	Spoke Guard 20" Flower	1
3	1522864	Spoke Guard 22" Flower	1

Tipping Tube - Back Bag

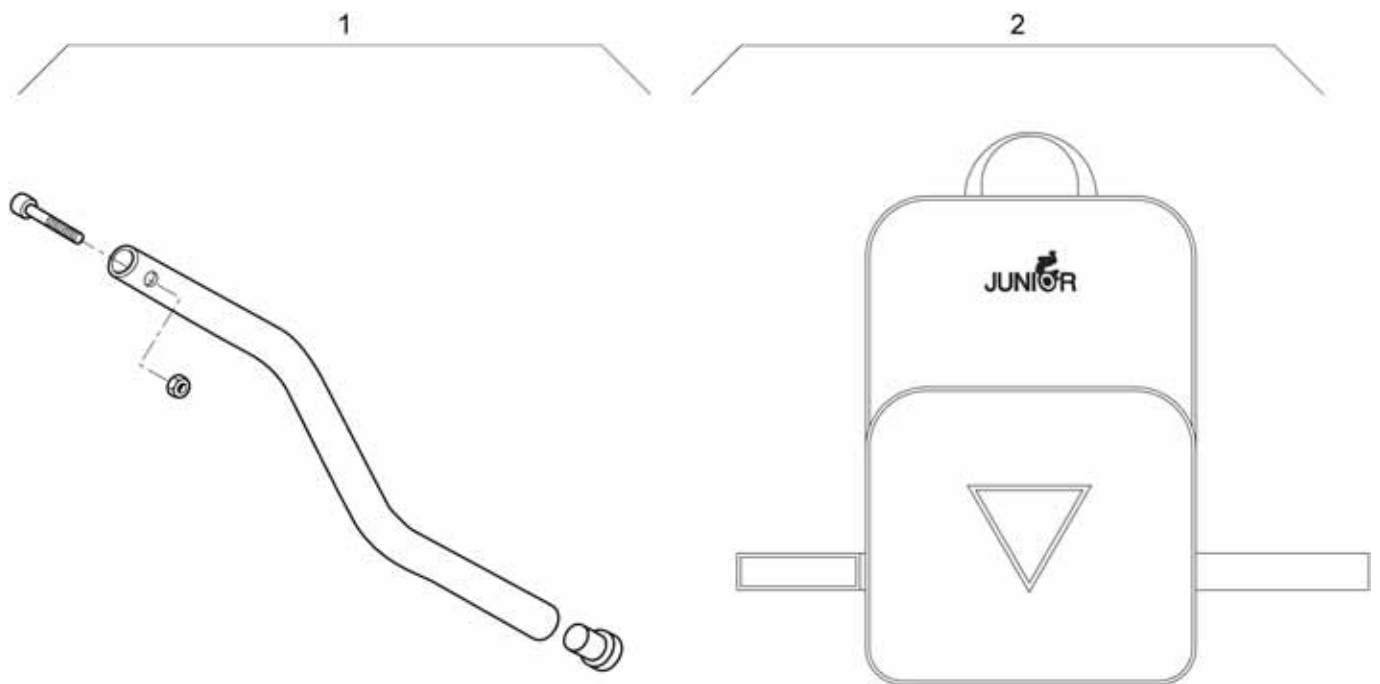


Pos.	Part number	Description	Qty
1	0572-05143	Tipping Tube	1
2	5108159	BACKPACK, WHEELER	1

Transparent Tray < 10/2007

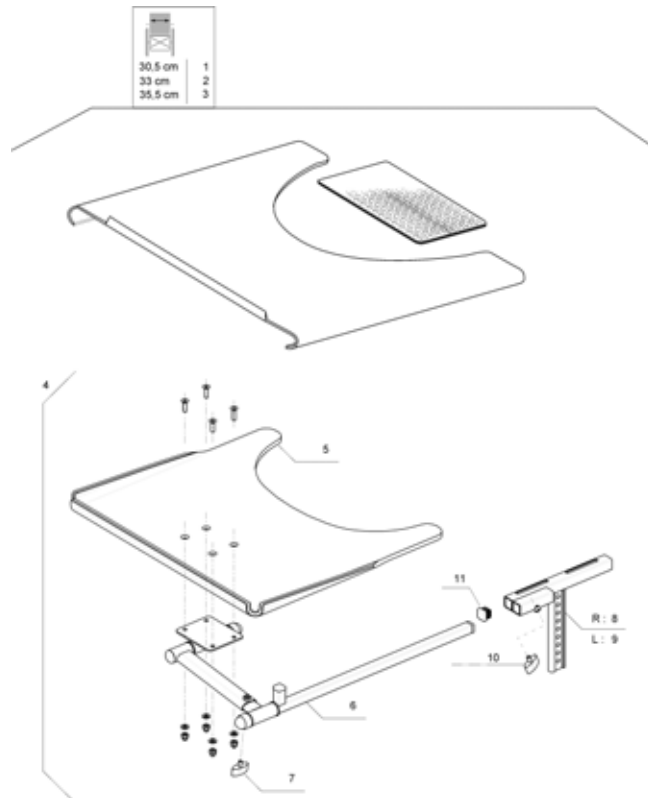

Pos.	Part number	Description	Qty
4	1493067	Evolutive Transparent Tray Assy	1
5	1492656	Evolutive Transparent Tray	1
6	1488146	Evolutive Transparent Tray Fixation	1
7	L31001	Wing ScREW M6X10	1
8	1492011-0601	Lh T Support Armrest Tray A3J	1
9	1492012-0601	Rh T Support Armrest Tray A3J	1
10	L10024	Wing Thumb Screw M6 X 16	1
11	L03010	20 Mm Square Cap	1

Junior Transparent Trays

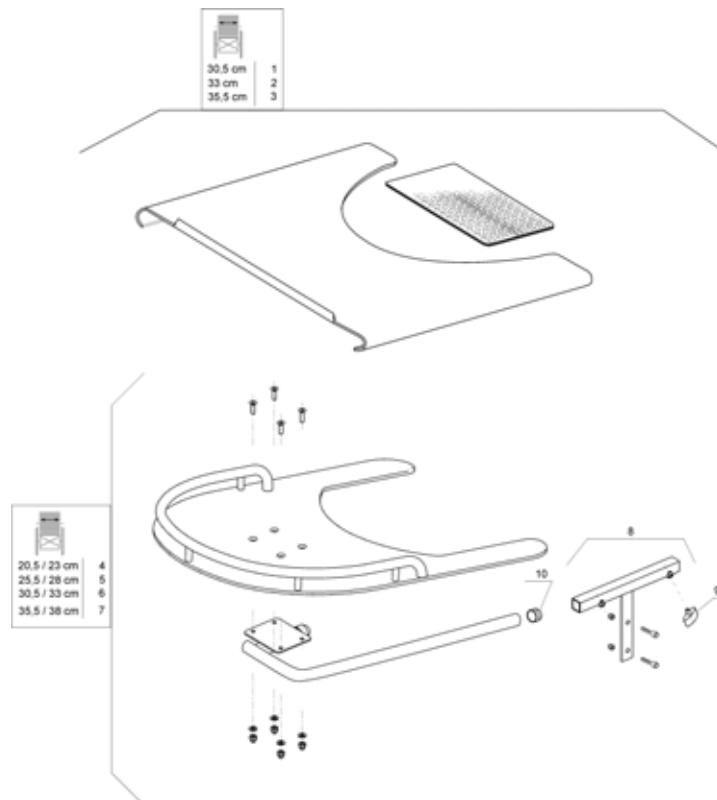

Pos.	Part number	Description	Qty
1	1492757	Transparent Tray 30,5 cm	1
2	1492758	Transparent Tray 33 cm	1
3	1492759	Transparent Tray 35,5 cm	1
4	1513253	Action 3 Junior Transparent Tray w 20,5 /23 cm	1
5	1513254	Action 3 Junior Transparent Tray w 25,5 /28 cm	1
6	1513255	Action 3 Junior Transparent Tray w 30,5 /33 cm	1
7	1513256	Action 3 Junior Transparent Tray w 35,5 /38 cm	1
8	1513579	T Support Armrest Tray Action 3 Junior	1
9	L10024	Wing Thumb Screw M6 X 16	1



Yes, you can.®



Belgium & Luxemburg:

Invacare nv, Autobaan 22, B-8210 Loppem
Tel: (32) (0)50 83 10 10, Fax: (32) (0)50 83 10 11
belgium@invacare.com

Danmark:

Invacare A/S, Sdr. Ringvej 37, DK-2605 Brøndby
Tel: (45) (0)36 90 00 00, Fax: (45) (0)36 90 00 01
denmark@invacare.com

Deutschland:

Invacare Aquatec GmbH, Alemannenstraße 10, D-88316 Isny
Tel: (49) (0)75 62 7 00 0, Fax: (49) (0)75 62 7 00 66
info@invacare-aquatec.com

European Distributor Organisation:

Invacare, Kleiststraße 49, D-32457 Porta Westfalica
Tel: (49) (0)57 31 754 540, Fax: (49) (0)57 31 754 541
edo@invacare.com

España:

Invacare SA, c/Areny s/n, Polígon Industrial de Celrà, E-17460 Celrà (Girona)
Tel: (34) (0)972 49 32 00, Fax: (34) (0)972 49 32 20
contactsp@invacare.com

France:

Invacare Poirier SAS, Route de St Roch, F-37230 Fondettes
Tel: (33) (0)2 47 62 64 66, Fax: (33) (0)2 47 42 12 24
contactfr@invacare.com

Ireland:

Invacare Ireland Ltd, Unit 5 Seatown Business Campus, Seatown Road, Swords, County Dublin - Ireland
Tel: (353) 1 810 7084, Fax: (353) 1 810 7085
ireland@invacare.com

Italia:

Invacare Mecc San s.r.l., Via dei Pini 62, I-36016 Thiene (VI)
Tel: (39) 0445 38 00 59, Fax: (39) 0445 38 00 34
italia@invacare.com

Nederland:

Invacare BV, Celsiusstraat 46, NL-6716 BZ Ede
Tel: (31) (0)318 695 757, Fax: (31) (0)318 695 758
nederland@invacare.com, csede@invacare.com

Norge:

Invacare AS, Grensesvingen 9, Postboks 6230, Etterstad, N-0603 Oslo
Tel: (47) (0)22 57 95 00, Fax: (47) (0)22 57 95 01
norway@invacare.com, island@invacare.com

Österreich:

Mobitec Mobilitätshilfen GmbH, Herzog Odilostrasse 101, A-5310 Mondsee
Tel: (43) 6232 5535 0, Fax: (43) 6232 5535 4
office@mobitec-austria.com, austria@invacare.com

Portugal:

Invacare Lda, Rua Senhora de Campanhã 105, P-4369-001 Porto
Tel: (351) (0)225 1059 46147, Fax: (351) (0)225 1057 39
portugal@invacare.com

Sverige & Suomi:

Invacare AB, Fagerstagatan 9, S-163 91 Spånga
Tel: (46) (0)8 761 70 90, Fax: (46) (0)8 761 81 08
sweden@invacare.com, finland@invacare.com

Switzerland:

Mobitec Rehab AG, Benkenstrasse 260, CH-4108 Witterswil
Tel: (41) (0)61 487 70 80, Fax: (41) (0)61 487 70 81
office@mobitec-rehab.ch, switzerland@invacare.com

United Kingdom:

Invacare Ltd, South Road, Bridgend Industrial Estate, UK-Bridgend CF31 3PY
Tel: (44) (0)1656 664 321, Fax: (44) (0)1656 667 532
UK@invacare.com