

Spare Parts Catalogue

Version: 2010/11/10a EN



Invacare® *Action*³ *NG*



Yes, you can.®

General Information

For any order of spare parts the following information is required:

- Product type*
- Product name & model*
- Serial Number*
- Part-Number*
- Part-Description*

Exploded view drawings and code numbers are not contractual and subject to modification, in relation to Invacare® continuous product improvement policy to meet end users needs.

The reproduction or use of this work, or of any substantial part of this work, in any form or by any electronic, mechanical or other means, including photocopying, scanning and recording, and in any information storage and retrieval system, including electronic transmission, is forbidden without the prior written permission of Invacare® International Sarl.



Yes, you can.®

TABLE OF CONTENTS

Information

Abbreviations	7
---------------------	---

Chassis

Side Frame	8
Folding System	9

Seat

Seat Upholstery	10
Padded Seats Upholsteries	11
Adjustable Depth Seat Upholstery	12
Padded Seats Upholsteries And Mistral Cushions	13
Anatomic Seat Adjustable	14

Backrest

Fixed Backrest	15
Fixed Backrest Height Push Handle	16
Additional Push Handle	17
Fixed Backrest, Height Adjustable, Push Handle	18
Fixed Backrest 35 Cm Xtra	19
Adjustable Backrest From -6° To +24°	20
Recliner Backrest 0° To 30°, Ratchet	21
Recliner Backrest 0° To 30°, Gaz Spring	22
Recliner Backrest 0° To 30° With Adjustable Handle, Gaz Spring	23
Fold down backrest	24
Laguna backrests	25
Rea Backrest Upholstery	26
Lombar support padded back upholstery	27
Contour adjustable padded back upholstery	28
Backrest Upholstery For Height Adjustable Backrest	29
Contour adjustable nylon type back upholstery	30
Rea contour adjustable back upholstery	31
Comfort Upholstery	32

Legrests

Hanger fixed 80°	33
Legrest 90°, fixed	34
Assembly Legrest, Fixed + Ajustable Footrest	35
Adjustable Legrest	36
Calf Pad	37
Amputee Legrest	38

Footrests

Footrest	39
Footrest with shorter support	40
Angle adjustable footrest	41
Angle and depth adjustable footrest	42
Angle and depth adjustable footrest with shorter support	44



Yes, you can.®

TABLE OF CONTENTS

One piece footboard	45
Siderests	
Armrest n°1 - Removable and height adjustable	46
Swing Away & Detachable Armrest 2	47
Height Adjustable Armrest 3	49
Height Adjustable Comfort Armrest	51
Height Adjustable Armrest 4	52
Hemi Dartex Armpad Multiposition	53
Mudguard Xtra	54
Armpads raiser kit	55
Front Wheels	
Forks	56
Forks supports	57
Front wheels	58
Front wheels Compact	59
Rear Wheels	
24" Standard Wheels	60
24" Heavy Duty Wheels	61
24" Standard Wheels accessories	62
22" Standard Wheels	63
22" Heavy Duty Wheels	64
22" Standard Wheels accessories	65
20" Standard Wheels Compact	66
20" Standard Wheels accessories Compact	67
24" / 22" Hub brake wheels	68
Rear Wheel Supports	69
12" Rear Wheels	70
Rear wheel support transit	71
Pump	72
Brakes	
Push Brakes : Standard And With Extension	73
Push brake Transit	74
Attendant Hub Brake	75
24" / 22" Hub brake wheels	68
Oad Wire Control Brake	76
Safety Options	
Lapt belts	77
Rear Anti Tipper And Tipping Tube	78
Anti-tip	79
Reflectors	80
Transfer Kit	81
Seating Options	
Standard Headrest	82



Yes, you can.®

TABLE OF CONTENTS

Comfort Headrest	83
Cushions	84
Cushions Visco	85
Trunk Support Comfort Dartex	86
Calf Strap	87
Heels strap	88
Back Brace	89


Propulsion Options

One arm drive	90
Hand Control For Oad By Lever	91
Oad By Lever	92
Wheels For Oad By Lever	93
Dual Amputee Position	94

Others Options & Accessories

Tipping tube	95
Cane holder	96
Dripstand	97
Transparent Tray	98
Comfort Tray	99


Abbreviations

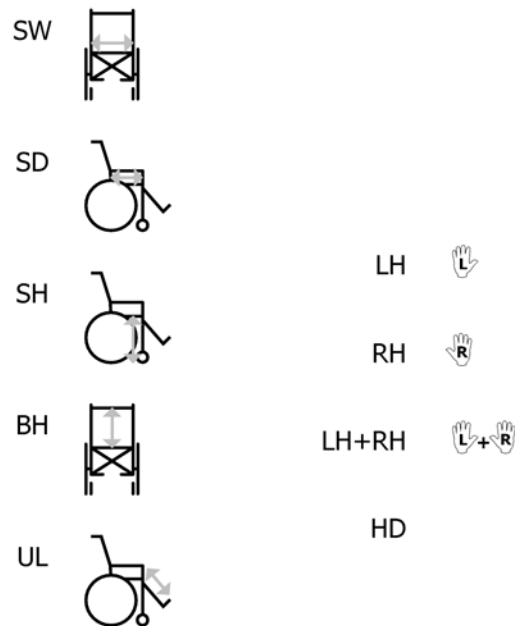

19P5133721


Pos.	Part number	Description	Valid from → until	Qty
-	SW	Seat width (in cm)		1
-	SD	Seat depth (in cm)		1
-	SH	Seat height (in cm)		1
-	BH	Backrest height (in cm)		1
-	UL	Knee-to-heel height (in cm)		1
-	LH	Left side (sitting in the wheelchair)		1
-	RH	Right side (sitting in the wheelchair)		1
-	LH+RH	An assembly of a left and a right part, referred to as "pair"		1
-	HD	Heavy duty version		1

Side Frame

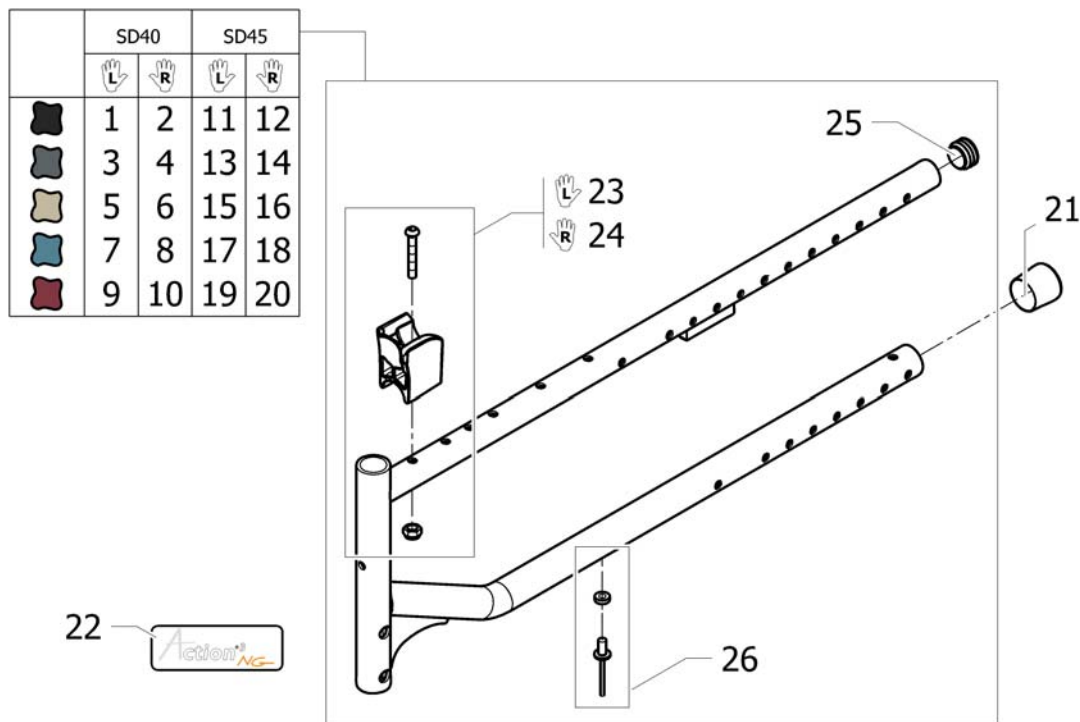

Pos.	Part number	Description	Valid from → until	Qty
1	1449982-0609	Side Frame Lh SD 40Cm Anthracite Black		1
2	1449981-0609	Side Frame Rh SD 40Cm Anthracite Black		1
3	1449982-0615	Side Frame Lh SD 40Cm Slate Grey		1
4	1449981-0615	Side Frame Rh SD 40Cm Slate Grey		1
5	1449982-0616	Side Frame Lh SD 40Cm Ice Brown		1
6	1449981-0616	Side Frame Rh SD 40Cm Ice Brown		1
7	1449982-0617	Side Frame Lh SD 40Cm Ocean Blue		1
8	1449981-0617	Side Frame Rh SD 40Cm Ocean Blue		1
9	1449982-0607	Side Frame Lh D 40Cm Dark Red		1
10	1449981-0607	Side Frame Rh D 40Cm Dark Red		1
11	1449984-0609	Side Frame Lh SD 45Cm Anthracite Black		1
12	1449983-0609	Side Frame Rh SD 45Cm Anthracite Black		1
13	1449984-0615	Side Frame Lh SD 45Cm Slate Grey		1
14	1449983-0615	Side Frame Rh SD 45Cm Slate Grey		1
15	1449984-0616	Side Frame Lh SD 45Cm Ice Brown		1
16	1449983-0616	Side Frame Rh SD 45Cm Ice Brown		1
17	1449984-0617	Side Frame Lh SD 45Cm Ocean Blue		1
18	1449983-0617	Side Frame Rh SD 45Cm Ocean Blue		1
19	1449984-0607	Side Frame Lh D 45Cm Dark Red		1
20	1449983-0607	Side Frame Rh D 45Cm Dark Red		1
21	0021N05386	Frame cap black 1		1
22	1533157	Sticker Action3NG		1
23	1434128	Crossbar Support Kit Clipable Left		1
24	1434127	Crossbar Support Kit Clipable Right		1
25	1418112	Black cap to be inserted Ø22x15		1
26	1431304	Cross brace rivet & washer kit		1


Folding System

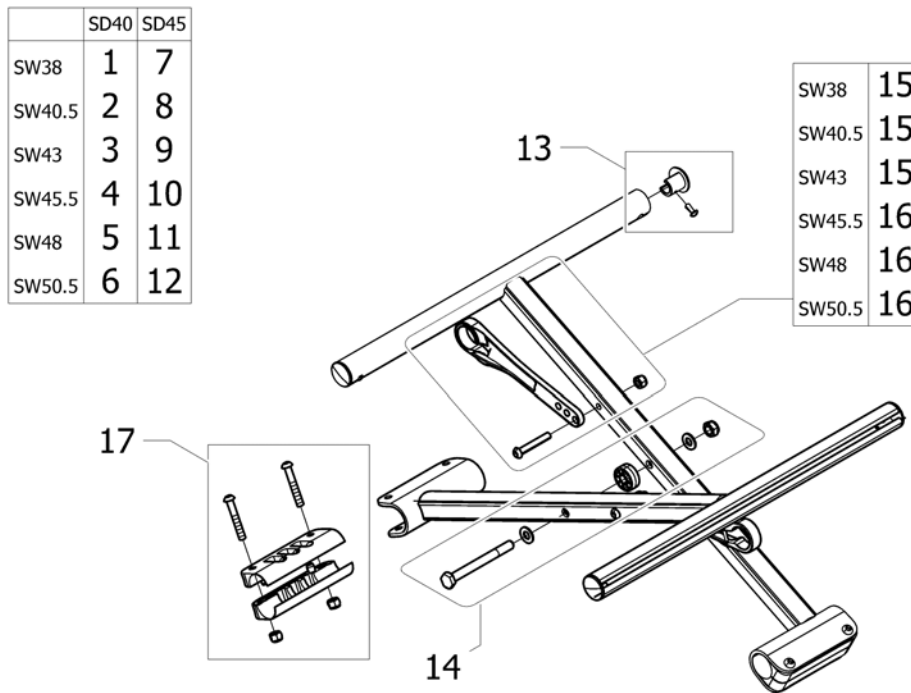

1895133402

Pos.	Part number	Description	Valid from → until	Qty
1	1530223	Cross Brace Complete Mechanism SW 38Cm X SD 40Cm		1
2	1530224	Cross Brace Complete Mechanism SW 40,5Cm X SD 40Cm		1
3	1530225	Cross Brace Complete Mechanism SW 43Cm X SD 40Cm		1
4	1530226	Cross Brace Complete Mechanism SW 45,5Cm X SD 40Cm		1
5	1530227	Cross Brace Complete Mechanism SW 48Cm X SD 40Cm		1
6	1530228	Cross Brace Complete Mechanism SW 50,5Cm X SD 40Cm		1
7	1530607	Cross Brace Complete Mechanism SW 38Cm X SD 45Cm		1
8	1530608	Cross Brace Complete Mechanism SW 40,5Cm X SD 45Cm		1
9	1530609	Cross Brace Complete Mechanism SW 43Cm X SD 45Cm		1
10	1530610	Cross Brace Complete Mechanism SW 45,5Cm X SD 45Cm		1
11	1530611	Cross Brace Complete Mechanism SW 48Cm X SD 45Cm		1
12	1530612	Cross Brace Complete Mechanism SW 50,5Cm X SD 45Cm		1
13	1431307	Seat end cap kit		2
14	1532556	Cross brace axle kit		1
15	1431306	Cross brace strut SW38-43		2
16	1430872	Cross brace strut SW45.5-50.5		2
17	1431305	Crossbrace sleeve		2

Seat Upholstery

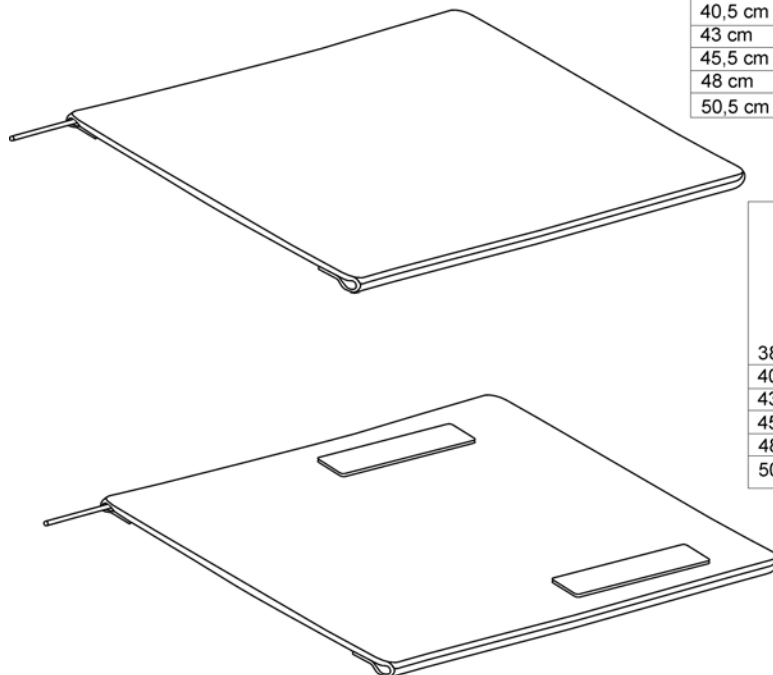



	40 cm	45 cm
38 cm	1	7
40,5 cm	2	8
43 cm	3	9
45,5 cm	4	10
48 cm	5	11
50,5 cm	6	12

	40 cm	45 cm
38 cm	13	19
40,5 cm	14	20
43 cm	15	21
45,5 cm	16	22
48 cm	17	23
50,5 cm	18	24

Pos.	Part number	Description	Valid from → until	Qty
1	1428840	Basic Seat Upholstery W 38Cm X D 40Cm		1
2	1428841	Basic Seat Upholstery W 40,5Cm X D 40Cm		1
3	1428842	Basic Seat Upholstery W 43Cm X D 40Cm		1
4	1428843	Basic Seat Upholstery W 45,5Cm X D 40Cm		1
5	1428844	Basic Seat Upholstery W 48Cm X D 40Cm		1
6	1428845	Basic Seat Upholstery W 50,5Cm X D 40Cm		1
7	1428846	Basic Seat Upholstery W 38Cm X D 45Cm		1
8	1428847	Basic Seat Upholstery W 40,5Cm X D 45Cm		1
9	1428848	Basic Seat Upholstery W 43Cm X D 45Cm		1
10	1428849	Basic Seat Upholstery W 45,5Cm X D 45Cm		1
11	1428850	Basic Seat Upholstery W 48Cm X D 45Cm		1
12	1428851	Basic Seat Upholstery W 50,5Cm X D 45Cm		1
13	1428852	Standard Seat Upholstery W 38Cm X D 40Cm		1
14	1428853	Standard Seat Upholstery W 40,5Cm X D 40Cm		1
15	1428854	Standard Seat Upholstery W 43Cm X D 40Cm		1
16	1428855	Standard Seat Upholstery W 45,5Cm X D 40Cm		1
17	1428856	Standard Seat Upholstery W 48Cm X D 40Cm		1
18	1428857	Standard Seat Upholstery W 50,5Cm X D 40Cm		1
19	1428858	Standard Seat Upholstery W 38Cm X D 45Cm		1
20	1428859	Standard Seat Upholstery W 40,5Cm X D 45Cm		1
21	1428860	Standard Seat Upholstery W 43Cm X D 45Cm		1
22	1428861	Standard Seat Upholstery W 45,5Cm X D 45Cm		1
23	1428862	Standard Seat Upholstery W 48Cm X D 45Cm		1
24	1428863	Standard Seat Upholstery W 50,5Cm X D 45Cm		1

Padded Seats Upholsteries

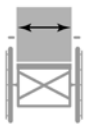
	40 cm	
38 cm	1	7
40,5 cm	2	8
43 cm	3	9
45,5 cm	4	10
48 cm	5	11
50,5 cm	6	12

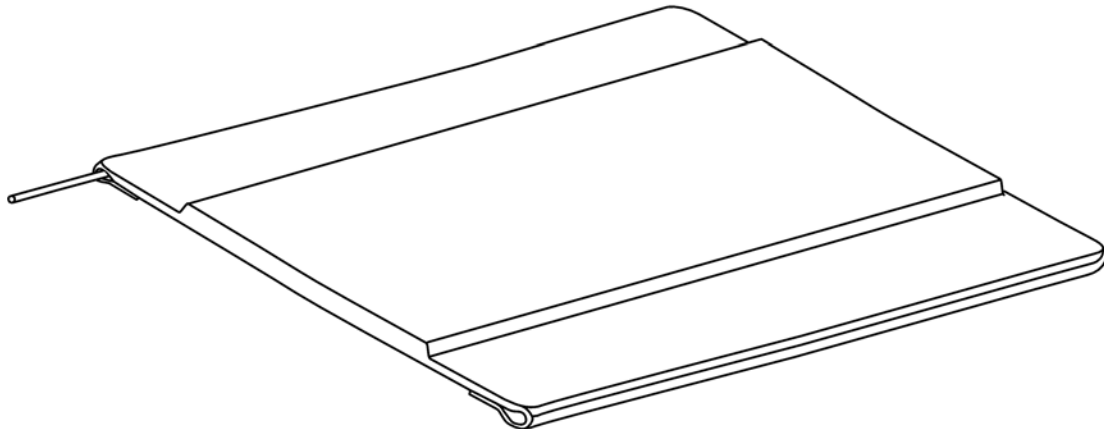

Pos.	Part number	Description	Valid from → until	Qty
1	1428864	Padded Seat Upholstery W 38Cm X D 40Cm		1
2	1428865	Padded Seat Upholstery W 40,5Cm X D 40Cm		1
3	1428866	Padded Seat Upholstery W 43Cm X D 40Cm		1
4	1428867	Padded Seat Upholstery W 45,5Cm X D 40Cm		1
5	1428868	Padded Seat Upholstery W 48Cm X D 40Cm		1
6	1428869	Padded Seat Upholstery W 50,5Cm X D 40Cm		1
7	1428870	Padded Seat Upholstery W 38Cm X D 45Cm		1
8	1428871	Padded Seat Upholstery W 40,5Cm X D 45Cm		1
9	1428872	Padded Seat Upholstery W 43Cm X D 45Cm		1
10	1428873	Padded Seat Upholstery W 45,5Cm X D 45Cm		1
11	1428874	Padded Seat Upholstery W 48Cm X D 45Cm		1
12	1428875	Padded Seat Upholstery W 50,5Cm X D 45Cm		1


Adjustable Depth Seat Upholstery

38 cm	1
40,5 cm	2
43 cm	3
45,5 cm	4
48 cm	5
50,5 cm	6

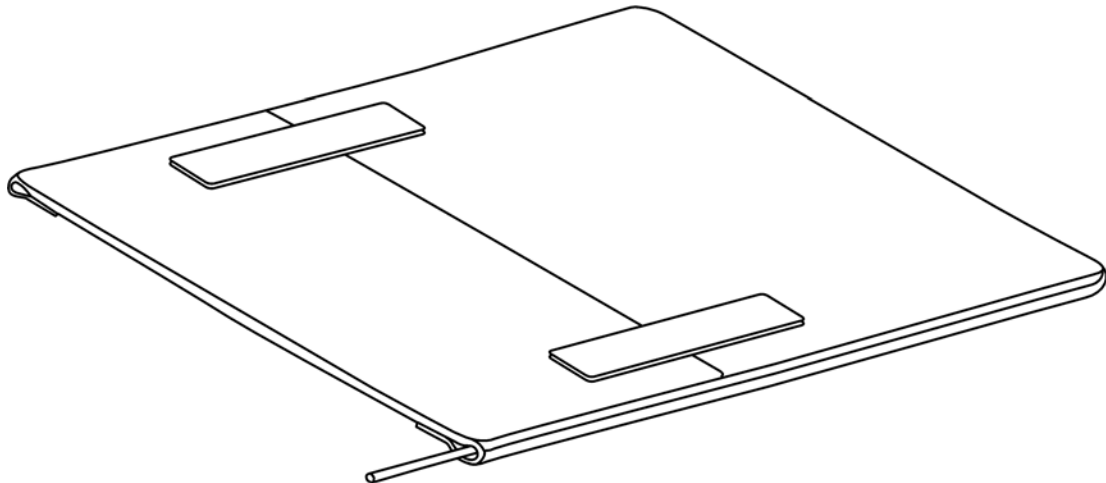

Pos.	Part number	Description	Valid from → until	Qty
1	1437580	Seat Upholstery 38/40-45, Sling Adj.		1
2	1437581	Seat Upholstery 40.5/40-45, Sling Adj.		1
3	1437582	Seat Upholstery 43/40-45, Sling Adj.		1
4	1437583	Seat Upholstery 45.5/40-45, Sling Adj.		1
5	1437584	Seat Upholstery 48/40-45, Sling Adj.		1
6	1437585	Seat Upholstery 50.5/40-45, Sling Adj.		1


Padded Seats Upholsteries And Mistral Cushions

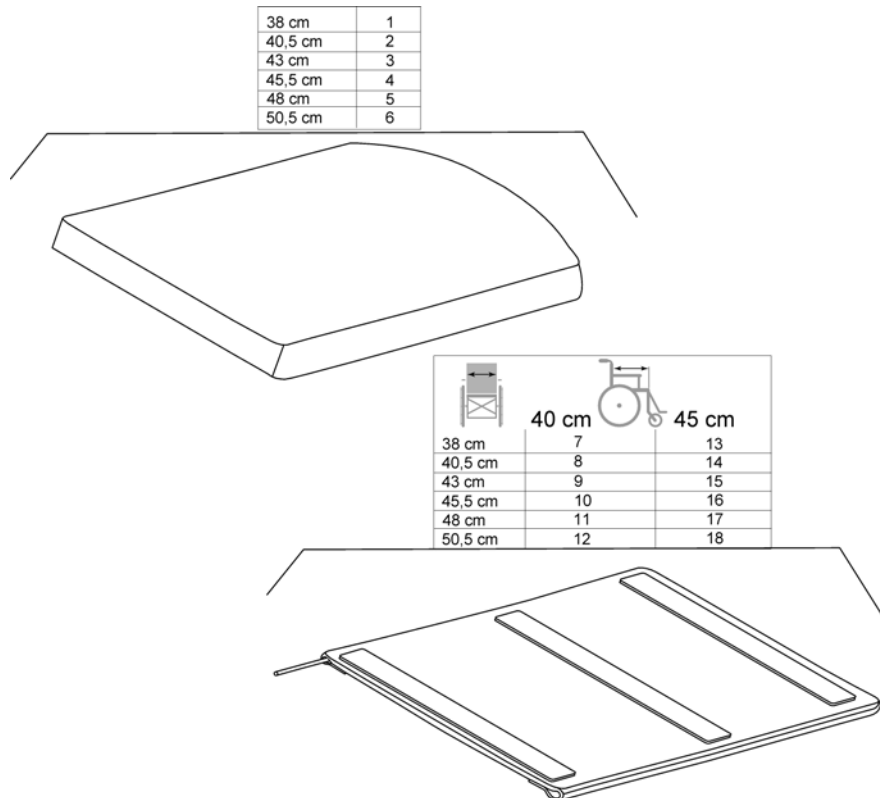


Pos.	Part number	Description	Valid from → until	Qty
1	1450115	Mistral 2 Layer Cushion W 38Cm X D 45Cm		1
2	1450116	Mistral 2 Layer Cushion W 40,5Cm X D 45Cm		1
3	1450117	Mistral 2 Layer Cushion W 43Cm X D 45Cm		1
4	1450118	Mistral 2 Layer Cushion W 45,5Cm X D 45Cm		1
5	1450119	Mistral 2 Layer Cushion W 48Cm X D 45Cm		1
6	1450120	Mistral 2 Layer Cushion W 50,5Cm X D 45Cm		1
7	1450067	Comfort Seat Upholstery W 38Cm X D 40Cm		1
8	1450068	Comfort Seat Upholstery W 40,5Cm X D 40Cm		1
9	1450069	Comfort Seat Upholstery W 43Cm X D 40Cm		1
10	1450070	Comfort Seat Upholstery W 45,5Cm X D 40Cm		1
11	1450071	Comfort Seat Upholstery W 48Cm X D 40Cm		1
12	1450072	Comfort Seat Upholstery W 50,5Cm X D 40Cm		1
13	1450123	Comfort Seat Upholstery W 38Cm X D 45Cm		1
14	1440918	Comfort Seat Upholstery W 40,5Cm X D 45 Cm		1
15	1440919	Comfort Seat Upholstery W 43Cm X D 45 Cm		1
16	1440920	Comfort Seat Upholstery W 45,5Cm X D 45 Cm		1
17	1450124	Comfort Seat Upholstery W 48Cm X D 45Cm		1
18	1440921	Comfort Seat Upholstery W 50,5Cm X D 45 Cm		1

Invacare® Action 3 NG | Seat

Anatomic Seat Adjustable

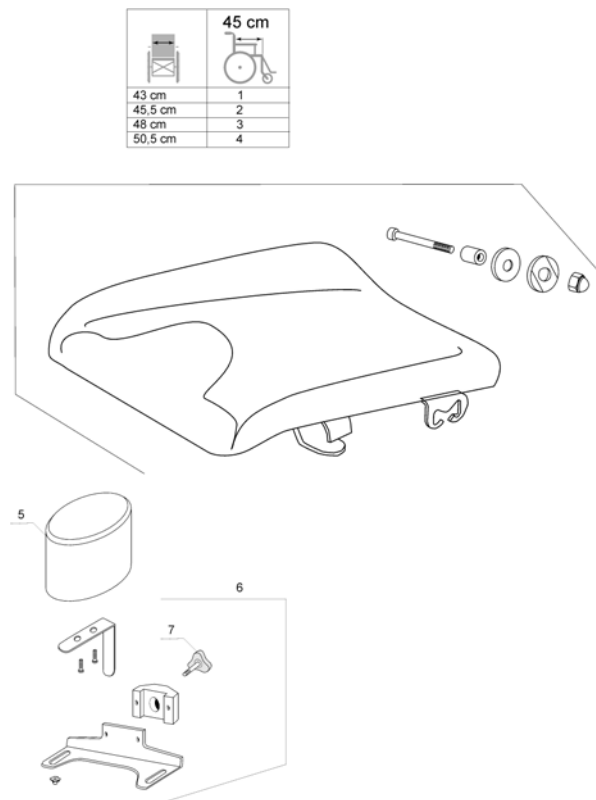


Pos.	Part number	Description	Valid from → until	Qty
1	1485159	Anatomic Seat Adjustable W 43Cm		1
2	1485160	Anatomic Seat Adjustable W 45,5Cm		1
3	1485161	Anatomic Seat Adjustable W 50,5Cm		1
4	1485535	Seat Frame Fixation		2
5	1444171	Abduction Pad Dartex		1
6	1485201	Abduction Support Kit Anatomic Seat		1
7	5221115	Knob M8 X 25		1

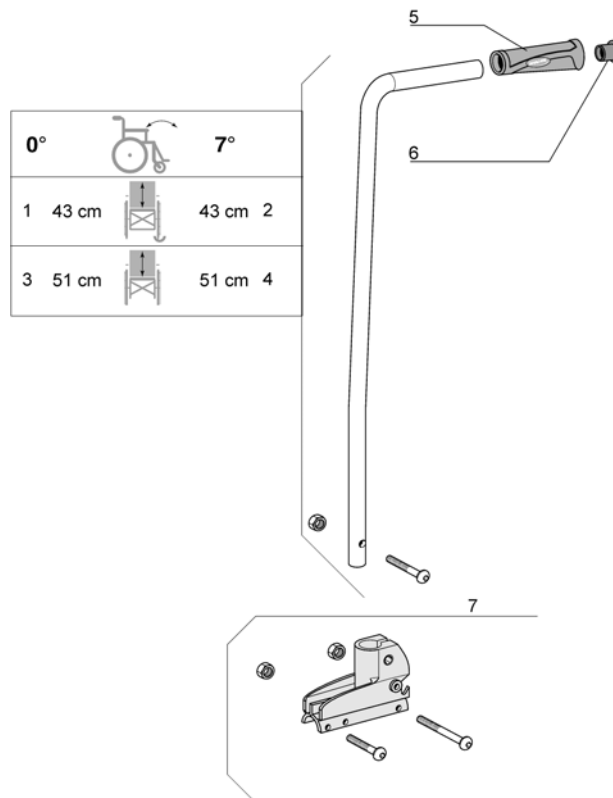
Fixed Backrest

Pos.	Part number	Description	Valid from → until	Qty
1	1430844	Backrest 0°/43Cm Cane Kit		2
2	1430846	Backrest 7°/43Cm Cane Kit		2
3	1430845	Backrest 0°/51Cm Cane Kit		2
4	1430847	Backrest 7°/51Cm Cane Kit		2
5	1425473	Invacare Push Handle <i>Replace 1435421</i>		2
6	1418112	Black cap to be inserted Ø22x15		2
7	1430848	Fixed Backrest Support Kit		2

Fixed Backrest Height Push Handle

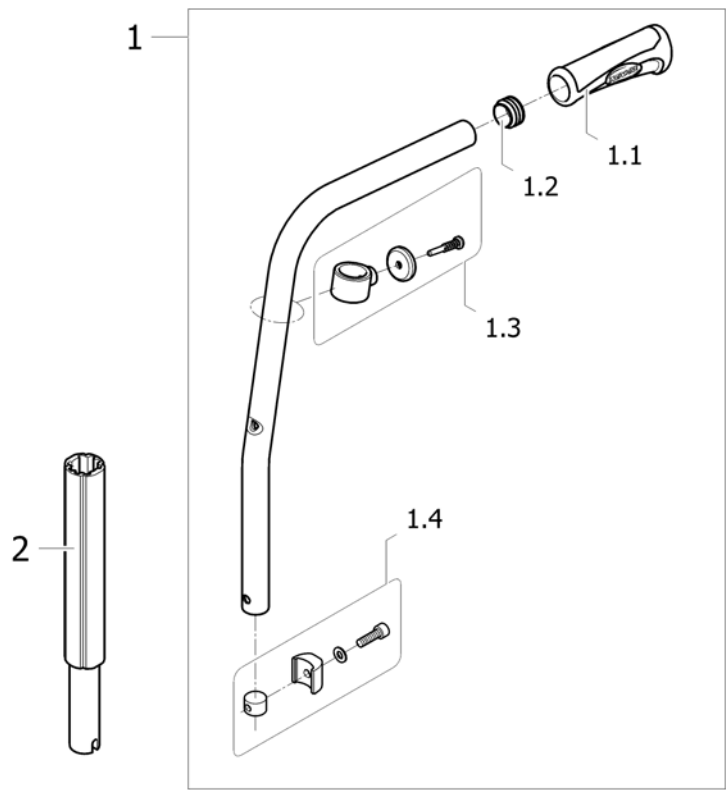


Pos.	Part number	Description	Valid from → until	Qty
1	1449802	A4 Backrest 7°/38Cm To 46 Cm Cane Kit		1
1.1	1435420	Handle backrest Ø22mm Invacare		2
1.2	1418112	Black cap to be inserted Ø22x15		2
1.3	1451816	A4 Fixed Backrest Upholstery Support Kit		2
1.4	1449803	A4 Fixed Backrest Fixation Kit		2
2	1538204	Backrest extension kit, removable		2

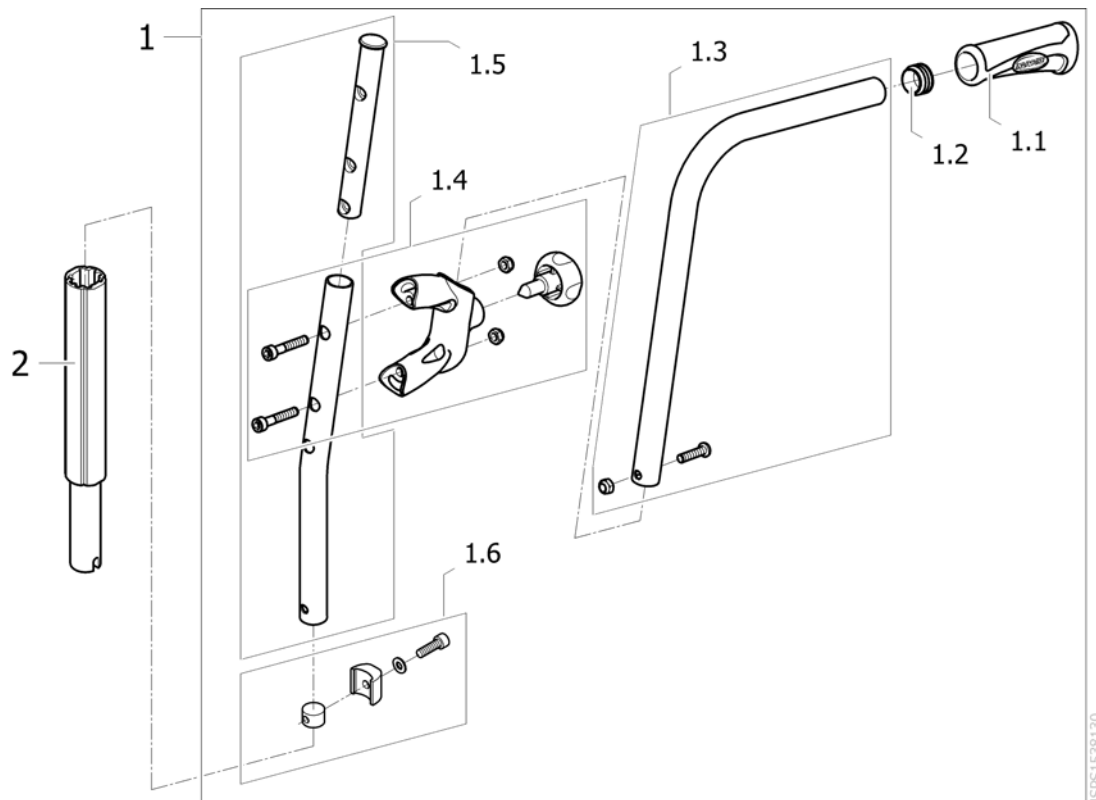Additional Push Handle

ISPS1538130


Pos.	Part number	Description	Valid from → until	Qty
1	1449804	Additional Push Handle Set		1
1.1	1435420	Handle backrest Ø22mm Invacare		1
1.2	1418112	Black cap to be inserted Ø22x15		1
1.3	1450574	Push Handle Cane Kit		1
1.4	1451818	Fixed Backrest Fixation Kit		1
1.5	1515698	Fixed backrest for additional push handle		1
1.6	1449803	A4 Fixed Backrest Fixation Kit		1
2	1538204	Backrest extension kit, removable		1

Fixed Backrest, Height Adjustable, Push Handle

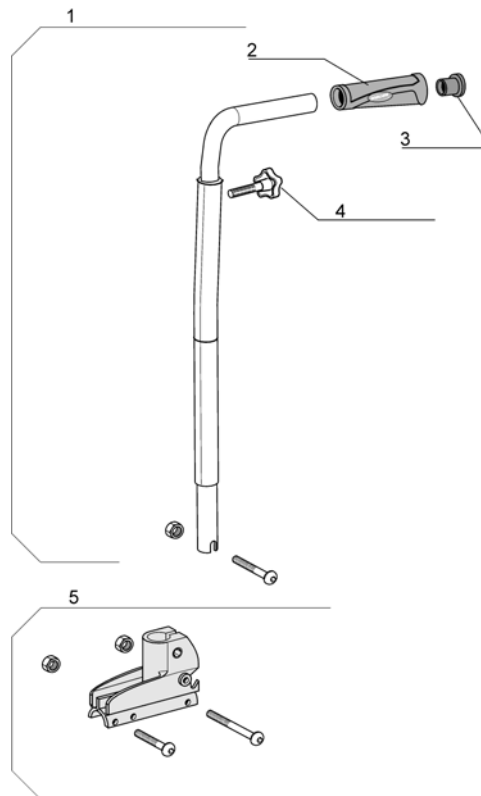


Pos.	Part number	Description	Valid from → until	Qty
1	1428641-0601	Adjustable Push Handle Backrest Complete (pair)		1
2	1425473	Invacare Push Handle <i>Replace 1435421</i>		2
3	1418112	Black cap to be inserted Ø22x15		2
4	0021-60029	Button Ø32 mm M6x16		2
5	1430848	Fixed Backrest Support Kit		2

Fixed Backrest 35 Cm Xtra

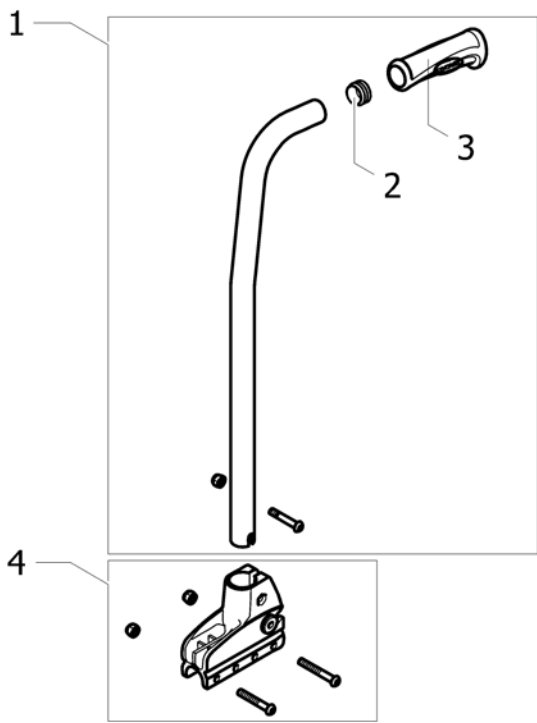


Pos.	Part number	Description	Valid from → until	Qty
1	1485911	Fixed Backrest Cane 35cm (pair)		1
2	1418112	Black cap to be inserted Ø22x15		2
3	1425473	Invacare Push Handle <i>Replace 1435421</i>		2
4	1430848	Fixed Backrest Support Kit		2

Adjustable Backrest From -6° To +24°

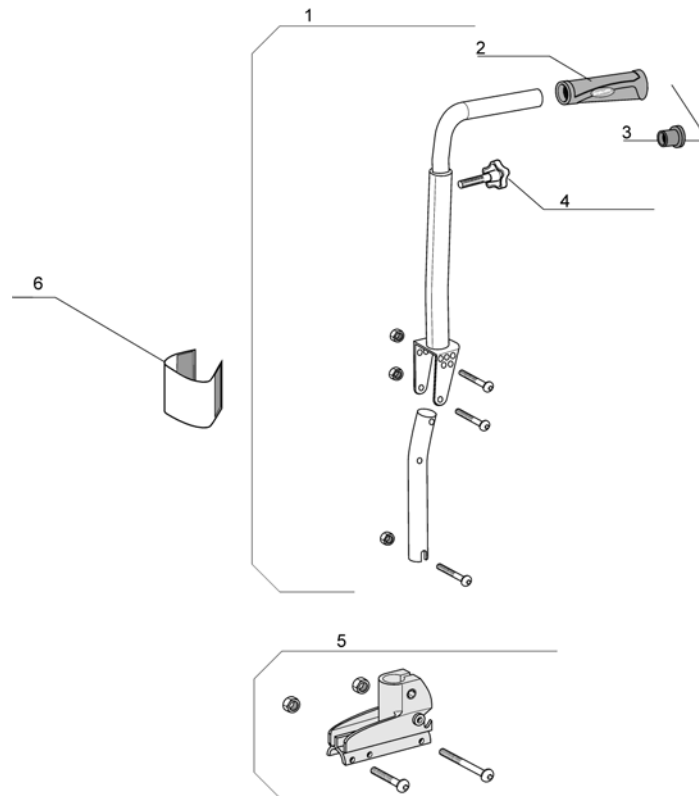


Pos.	Part number	Description	Valid from → until	Qty
1	1443021	-6°/+24° Backrest Cane Kit		2
2	1425473	Invacare Push Handle <i>Replace 1435421</i>		2
3	1418112	Black cap to be inserted Ø22x15		2
4	0021-60029	Button Ø32 mm M6x16		2
5	1430848	Fixed Backrest Support Kit		2
6	1442721	Protector Of Backrest Articulation		2

Recliner Backrest 0° To 30°, Ratchet

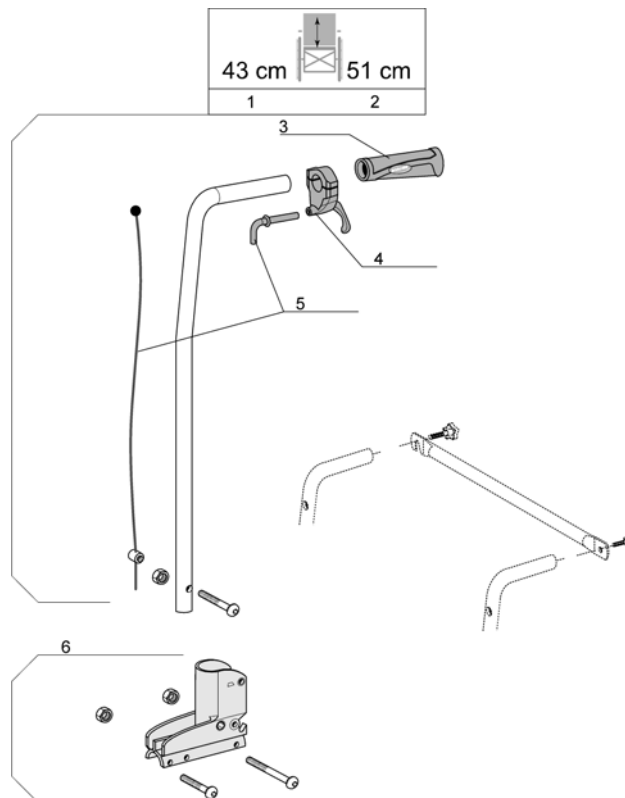

Pos.	Part number	Description	Valid from → until	Qty
1	1434491	Recliner backrest ratchet 0° to 30° cane kit SH35-37-39-41-43		2
2	1430849	Reclining Backrest Cane 51 Cm Complete		2
3	1425473	Invacare Push Handle <i>Replace 1435421</i>		2
4	1433747	Recliner Trigger		2
5	1430851	Recliner Cable Kit		2
6	1430850	Recliner Backrest Support Kit		2


Recliner Backrest 0° To 30°, Gaz Spring

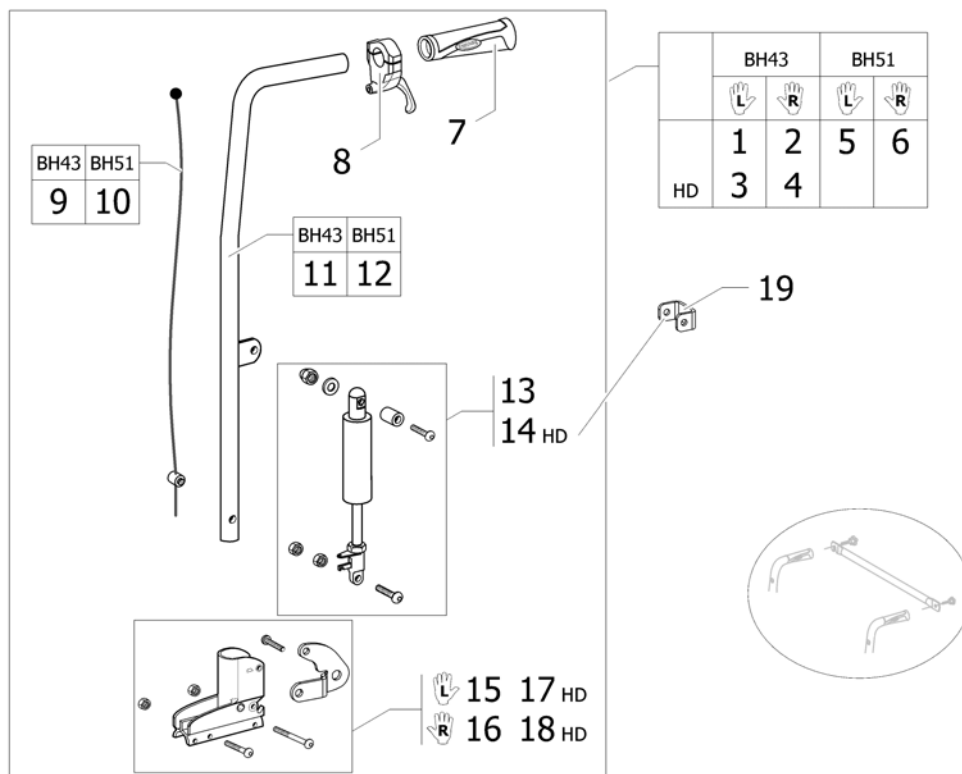

ISPS1538125


Pos.	Part number	Description	Valid from → until	Qty
1	1442539	Reclining Lh Kit Gas Strut Backrest 43 cm		1
2	1442538	Reclining Rh Kit Gas Strut Backrest 43 cm		1
3	1493416	Reclining Lh Kit Gas Strut Backrest H 43 Cm Hd		1
4	1493415	Reclining Rh Kit Gas Strut Backrest H 43 Cm Hd		1
5	1442540	Reclining Lh Kit Gas Strut Backrest 51 Cm		1
6	1442534	Reclining Rh Kit Gas Strut Backrest 51 Cm		1
7	1425473	Invacare Push Handle <i>Replace 1435421</i>		2
8	1433747	Recliner Trigger		2
9	1440937	Recliner Gas Cable 43 cm		2
10	1485871	Recliner Gas Cable 51 cm		2
11	1484055-0699	Recliner Gas Tube 43 cm		2
12	1484054-0699	Recliner Gas Tube 51 Cm		2
13	1528909	Kit, Gas Spring, Laguna2		2
14	1532596	Gas Strut Kit Standard Backrest Hd		2
15	1442513	Lh Recliner Gas Backrest Support Kit		2
16	1442507	Rh Recliner Gas Backrest Support Kit		2
17	1493418	Lh Recliner Gas Backrest Support Kit Hd		1
18	1493417	Rh Recliner Gas Backrest Support Kit Hd		1
19	1450992-0699	Head gas spring reinforcment		2

Recliner Backrest 0° To 30° With Adjustable Handle, Gaz Spring

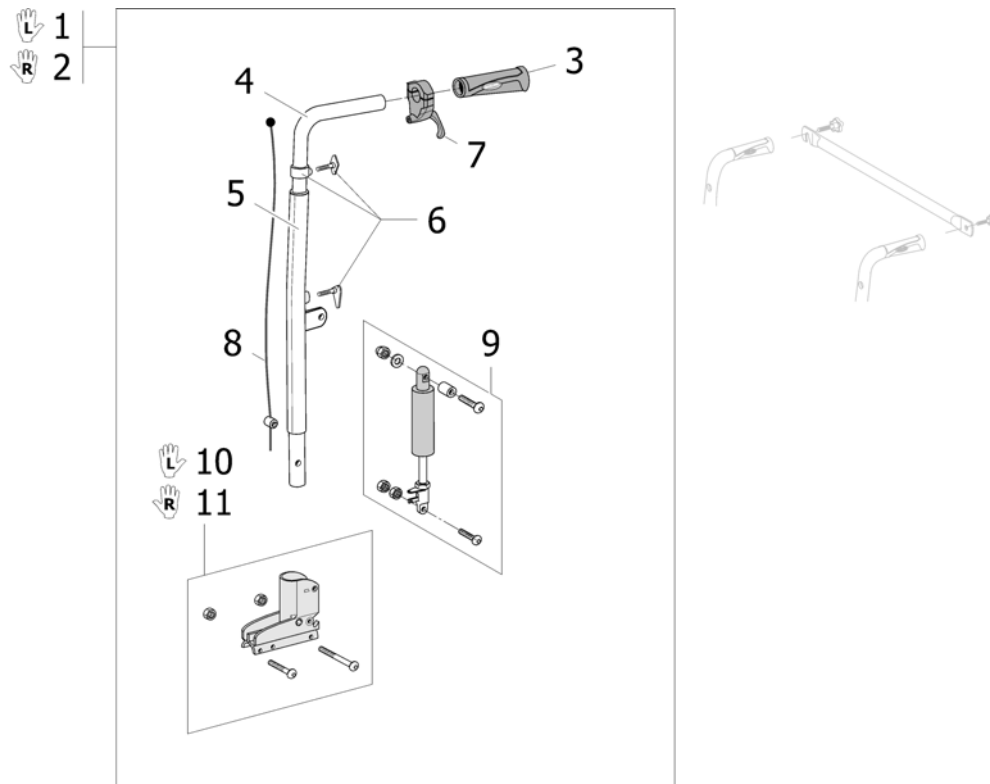


Pos.	Part number	Description	Valid from → until	Qty
1	1541894	Reclining Lh Kit Gas Strut Adjustable Bacrest		1
2	1541895	Reclining Rh Kit Gas Strut Adjustable Bacrest		1
3	1425473	Invacare Push Handle <i>Replace 1435421</i>		1
4	1448986-0010	Adjustable Push Handle Recliner Gas A4		1
5	1448985-0601	Adjustable Recliner Cane A4		1
6	1451819	Adjustable Backrest Push Handle Kit		1
7	1433747	Recliner Trigger		1
8	1451765	Adjustable Recliner Gas Cable Kit		1
9	1532595	Gas Strut Kit Standard Backrest		1
10	1442513	Lh Recliner Gas Backrest Support Kit		1
11	1442507	Rh Recliner Gas Backrest Support Kit		1

Fold down backrest

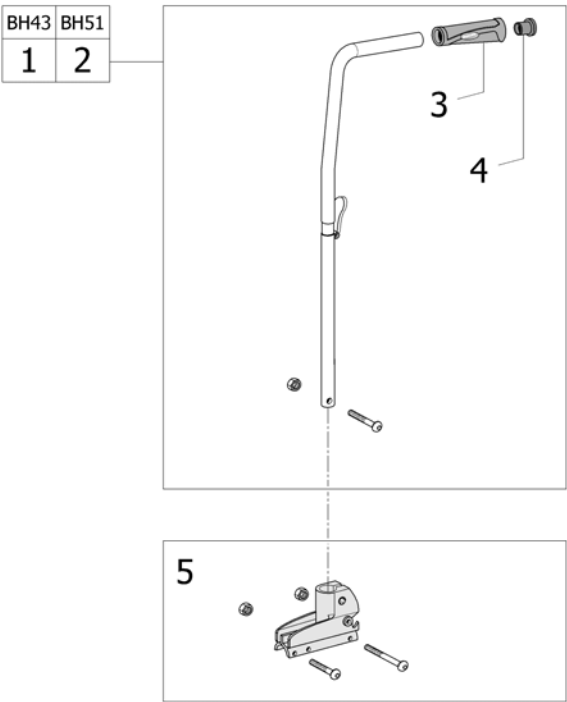

ISPS1494160

Pos.	Part number	Description	Valid from → until	Qty
1	1434492	Folding Backrest Cane 43 cm		2
2	1430852	Folding Backrest Cane 51 Cm		2
3	1425473	Invacare Push Handle <i>Replace 1435421</i>		2
4	1418112	Black cap to be inserted Ø22x15		2
5	1430848	Fixed Backrest Support Kit		2

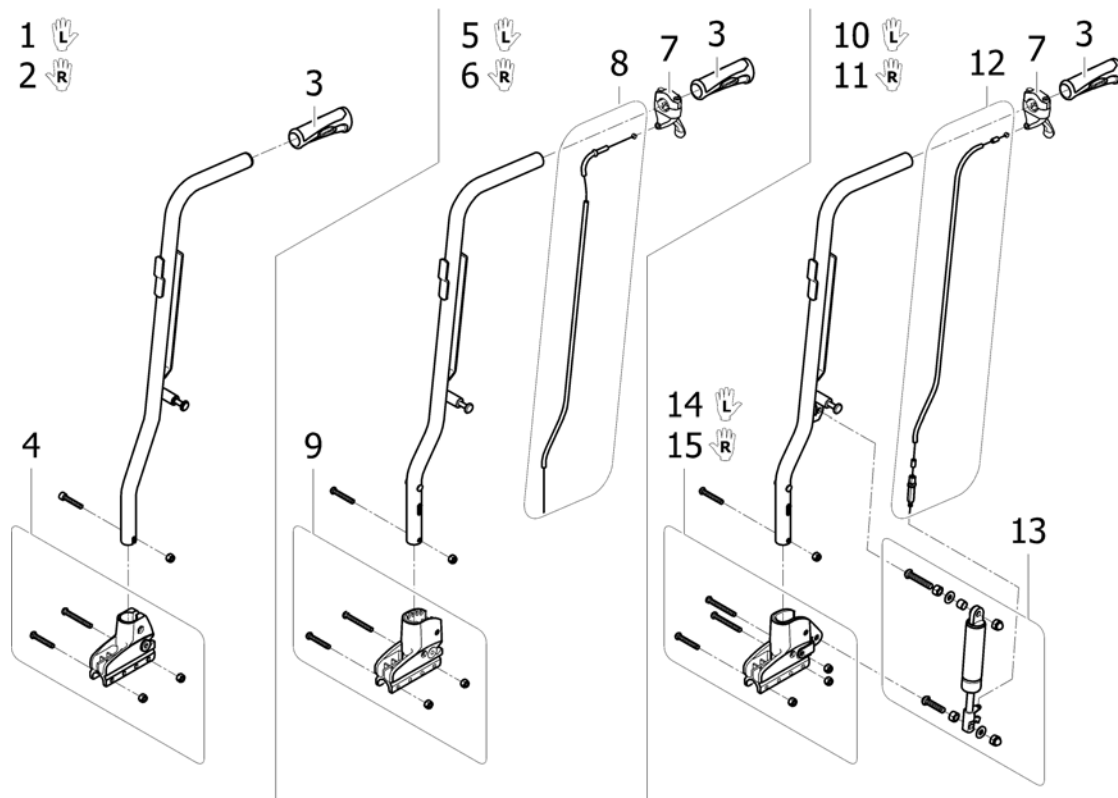
Laguna backrests

ISPS1514263

Pos.	Part number	Description	Valid from → until	Qty
1	1515663	Kit Backrest post Laguna fixed LH + handle + support		1
2	1515662	Kit Backrest post Laguna fixed RH + handle + support		1
3	1425473	Invacare Push Handle <i>Replace 1435421</i>		2
4	1430848	Fixed Backrest Support Kit		2
5	1515661	Kit Backrest post Laguna recliner ratchet LH + handle + trigger + cable + support		1
6	1515650	Kit Backrest post Laguna recliner ratchet RH + handle + trigger + cable + support		1
7	1433747	Recliner Trigger		2
8	1430851	Recliner Cable Kit		2
9	1430850	Recliner Backrest Support Kit		2
10	1541873	Kit Backrest post Laguna recliner gas LH + handle + trigger + cable + gas spring + support		1
11	1541872	Kit Backrest post Laguna recliner gas RH + handle + trigger + cable + gas spring + support		1
12	1485871	Recliner Gas Cable 51 cm		2
13	1528909	Kit, Gas Spring, Laguna2		2
14	1442513	Lh Recliner Gas Backrest Support Kit		1
15	1442507	Rh Recliner Gas Backrest Support Kit		1

Rea Backrest Upholstery


	BH43
SW38	1
SW40.5	2
SW43	3
SW45.5	4
SW48	5
SW50.5	6

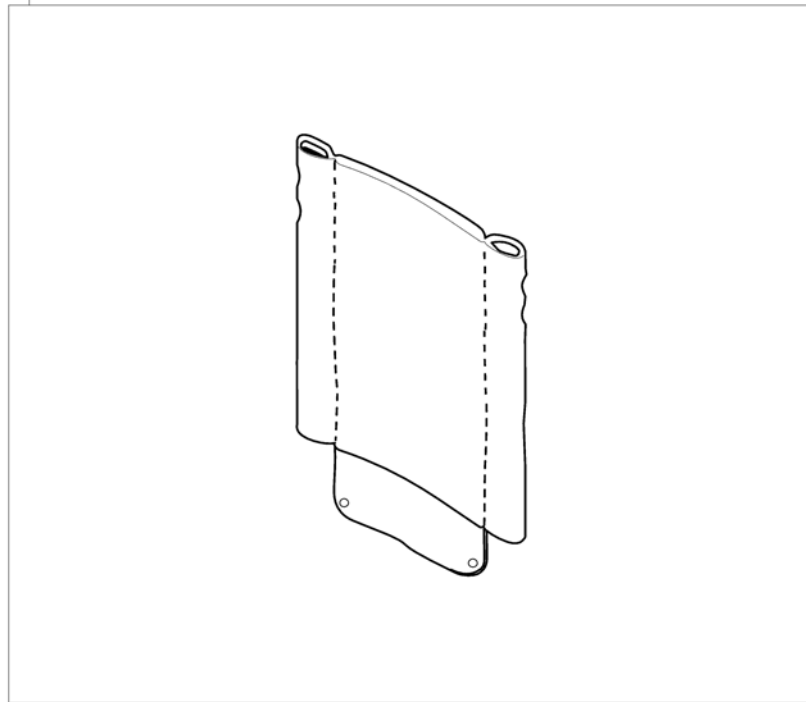

ISPS1543212

Pos.	Part number	Description	Valid from → until	Qty
1	1449809	Rea Backrest Upholstery W 38Cm X H 43Cm		1
2	1449810	Rea Backrest Upholstery W 40,5Cm X H 43Cm		1
3	1449811	Rea Backrest Upholstery W 43Cm X H 43Cm		1
4	1449812	Rea Backrest Upholstery W 45,5Cm X H 43Cm		1
5	1449813	Rea Backrest Upholstery W 48Cm X H 43Cm		1
6	1449814	Rea Backrest Upholstery W 50,5Cm X H 43Cm		1


Lombar support padded back upholstery

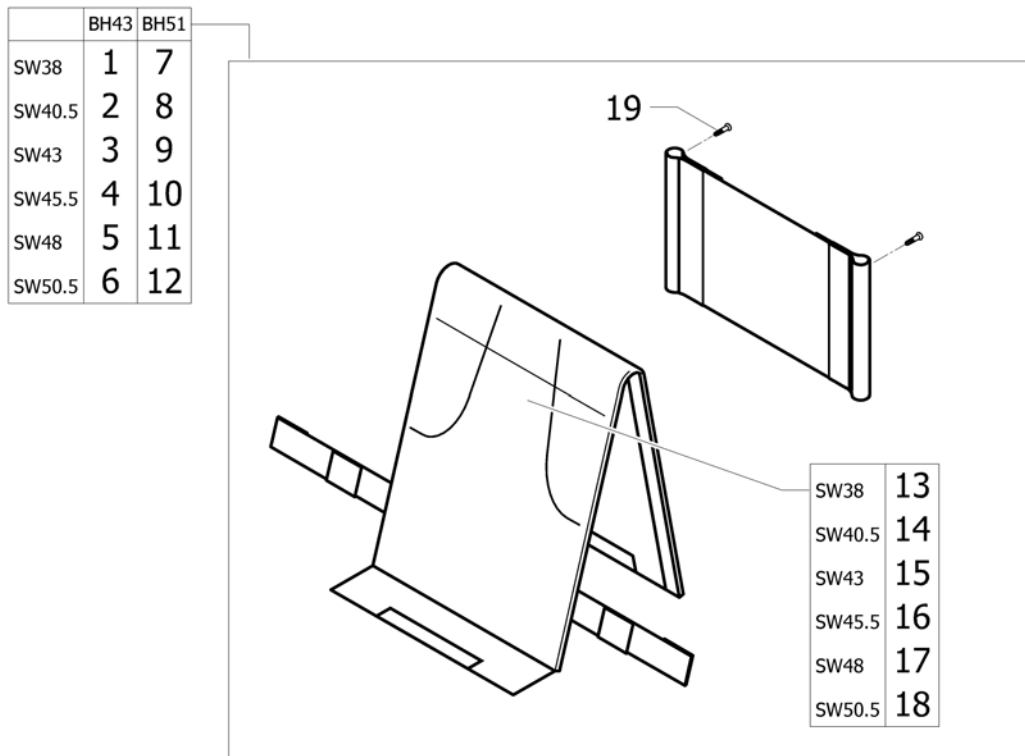

19P133337

Pos.	Part number	Description	Valid from → until	Qty
1	1532610	Lombar support padded back upholstery complete SW38 x BH43 cm		1
2	1532611	Lombar support padded back upholstery complete SW40,5 x BH43 cm		1
3	1532612	Lombar support padded back upholstery complete SW43 x BH43 cm		1
4	1532613	Lombar support padded back upholstery complete SW45,5 x BH43 cm		1
5	1532614	Lombar support padded back upholstery complete SW48 x BH43 cm		1
6	1532615	Lombar support padded back upholstery complete SW50,5 x BH43 cm		1
7	1532618	Lombar support padded back upholstery complete SW38 x BH51 cm		1
8	1532619	Lombar support padded back upholstery complete SW40,5 x BH51 cm		1
9	1532620	Lombar support padded back upholstery complete SW43 x BH51 cm		1
10	1532621	Lombar support padded back upholstery complete SW45,5 x BH51 cm		1
11	1532622	Lombar support padded back upholstery complete SW48 x BH51 cm		1
12	1532623	Lombar support padded back upholstery complete SW50,5 x BH51 cm		1
13	1530914	Shapped cover ActionNG SW38 cm		1
14	1530915	Shapped cover ActionNG SW40,5 cm		1
15	1531053	Shapped cover ActionNG SW43 cm		1
16	1531054	Shapped cover ActionNG SW45,5 cm		1
17	1531055	Shapped cover ActionNG SW48 cm		1
18	1531056	Shapped cover ActionNG SW50,5 cm		1
19	0029-40005	Nickel-plated Ø4,8x16 FBZ sheet-metal screw		2


Contour adjustable padded back upholstery

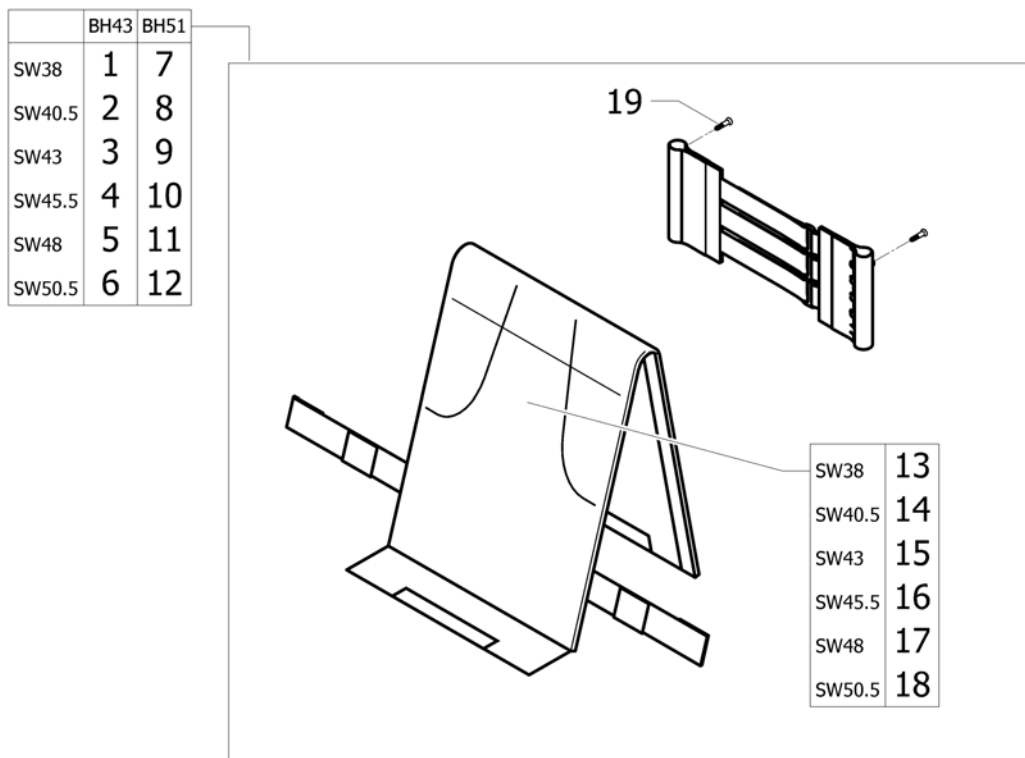

153133338

Pos.	Part number	Description	Valid from → until	Qty
1	1532667	Contour adjustable padded back upholstery complete SW38 x BH43 cm		1
2	1532686	Contour adjustable padded back upholstery complete SW40,5 x BH43 cm		1
3	1532687	Contour adjustable padded back upholstery complete SW43 x BH43 cm		1
4	1532779	Contour adjustable padded back upholstery complete SW45,5 x BH43 cm		1
5	1532780	Contour adjustable padded back upholstery complete SW48 x BH43 cm		1
6	1532781	Contour adjustable padded back upholstery complete SW50,5 x BH43 cm		1
7	1532784	Contour adjustable padded back upholstery complete SW38 x BH51 cm		1
8	1532785	Contour adjustable padded back upholstery complete SW40,5 x BH51 cm		1
9	1532786	Contour adjustable padded back upholstery complete SW43 x BH51 cm		1
10	1532787	Contour adjustable padded back upholstery complete SW45,5 x BH51 cm		1
11	1532788	Contour adjustable padded back upholstery complete SW48 x BH51 cm		1
12	1532789	Contour adjustable padded back upholstery complete SW50,5 x BH51 cm		1
13	1530914	Shapped cover ActionNG SW38 cm		1
14	1530915	Shapped cover ActionNG SW40,5 cm		1
15	1531053	Shapped cover ActionNG SW43 cm		1
16	1531054	Shapped cover ActionNG SW45,5 cm		1
17	1531055	Shapped cover ActionNG SW48 cm		1
18	1531056	Shapped cover ActionNG SW50,5 cm		1
19	0029-40005	Nickel-plated Ø4,8x16 FBZ sheet-metal screw		2


Backrest Upholstery For Height Adjustable Backrest

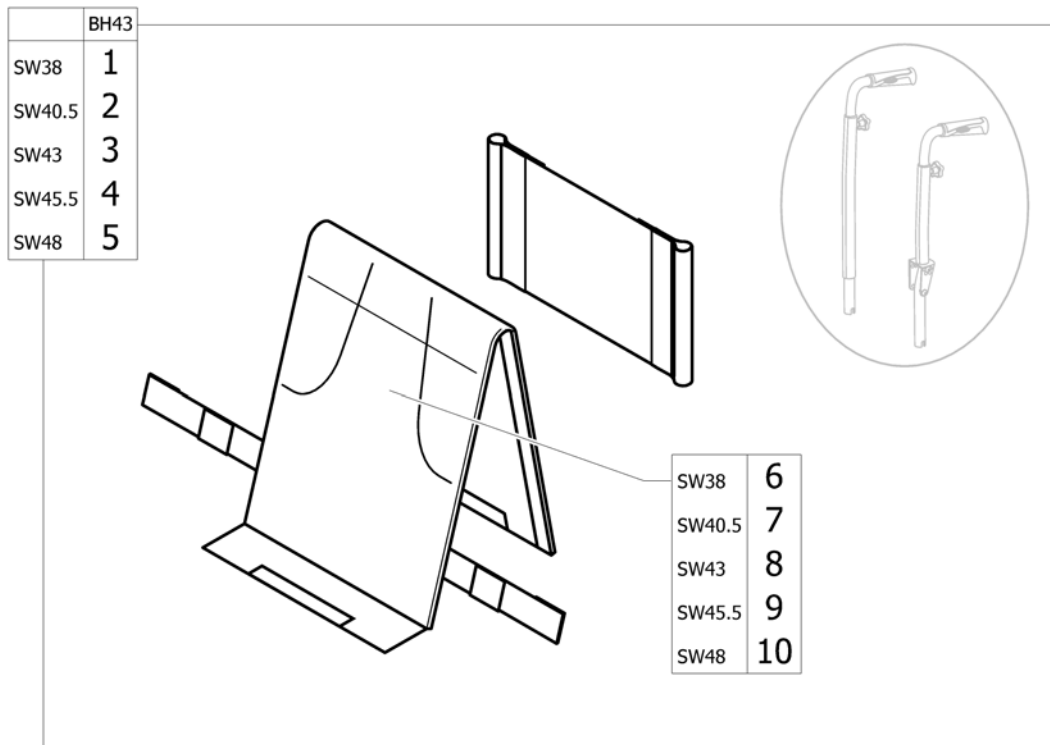

1591344017

Pos.	Part number	Description	Valid from → until	Qty
1	1533872	Lombar support padded back upholstery complete for height adjustable handle SW38 x BH43 cm		1
2	1533922	Lombar support padded back upholstery complete for height adjustable handle SW40,5 x BH43 cm		1
3	1533923	Lombar support padded back upholstery complete for height adjustable handle SW43 x BH43 cm		1
4	1533924	Lombar support padded back upholstery complete for height adjustable handle SW45,5 x BH43 cm		1
5	1533925	Lombar support padded back upholstery complete for height adjustable handle SW48 x BH43 cm		1
6	1530914	Shapped cover ActionNG SW38 cm		1
7	1530915	Shapped cover ActionNG SW40,5 cm		1
8	1531053	Shapped cover ActionNG SW43 cm		1
9	1531054	Shapped cover ActionNG SW45,5 cm		1
10	1531055	Shapped cover ActionNG SW48 cm		1


Contour adjustable nylon type back upholstery

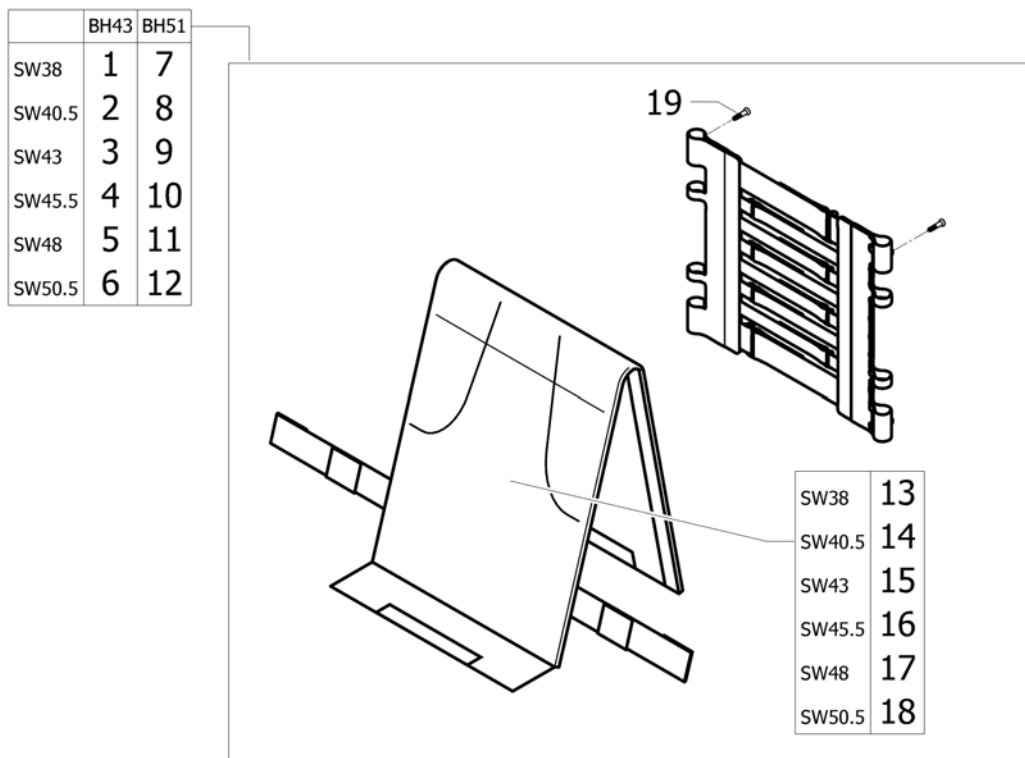

153133339


Pos.	Part number	Description	Valid from → until	Qty
1	1533860	Contour adjustable nylon type back upholstery complete SW38 x BH43 cm		1
2	1533861	Contour adjustable nylon type back upholstery complete SW40,5 x BH43 cm		1
3	1533862	Contour adjustable nylon type back upholstery complete SW43 x BH43 cm		1
4	1533863	Contour adjustable nylon type back upholstery complete SW45,5 x BH43 cm		1
5	1533864	Contour adjustable nylon type back upholstery complete SW48 x BH43 cm		1
6	1533865	Contour adjustable nylon type back upholstery complete SW50,5 x BH43 cm		1
7	1533866	Contour adjustable nylon type back upholstery complete SW38 x BH51 cm		1
8	1533867	Contour adjustable nylon type back upholstery complete SW40,5 x BH51 cm		1
9	1533868	Contour adjustable nylon type back upholstery complete SW43 x BH51 cm		1
10	1533869	Contour adjustable nylon type back upholstery complete SW45,5 x BH51 cm		1
11	1533870	Contour adjustable nylon type back upholstery complete SW48 x BH51 cm		1
12	1533871	Contour adjustable nylon type back upholstery complete SW50,5 x BH51 cm		1
13	1530914	Shapped cover ActionNG SW38 cm		1
14	1530915	Shapped cover ActionNG SW40,5 cm		1
15	1531053	Shapped cover ActionNG SW43 cm		1
16	1531054	Shapped cover ActionNG SW45,5 cm		1
17	1531055	Shapped cover ActionNG SW48 cm		1
18	1531056	Shapped cover ActionNG SW50,5 cm		1
19	0029-40005	Nickel-plated Ø4,8x16 FBZ sheet-metal screw		2

Rea contour adjustable back upholstery

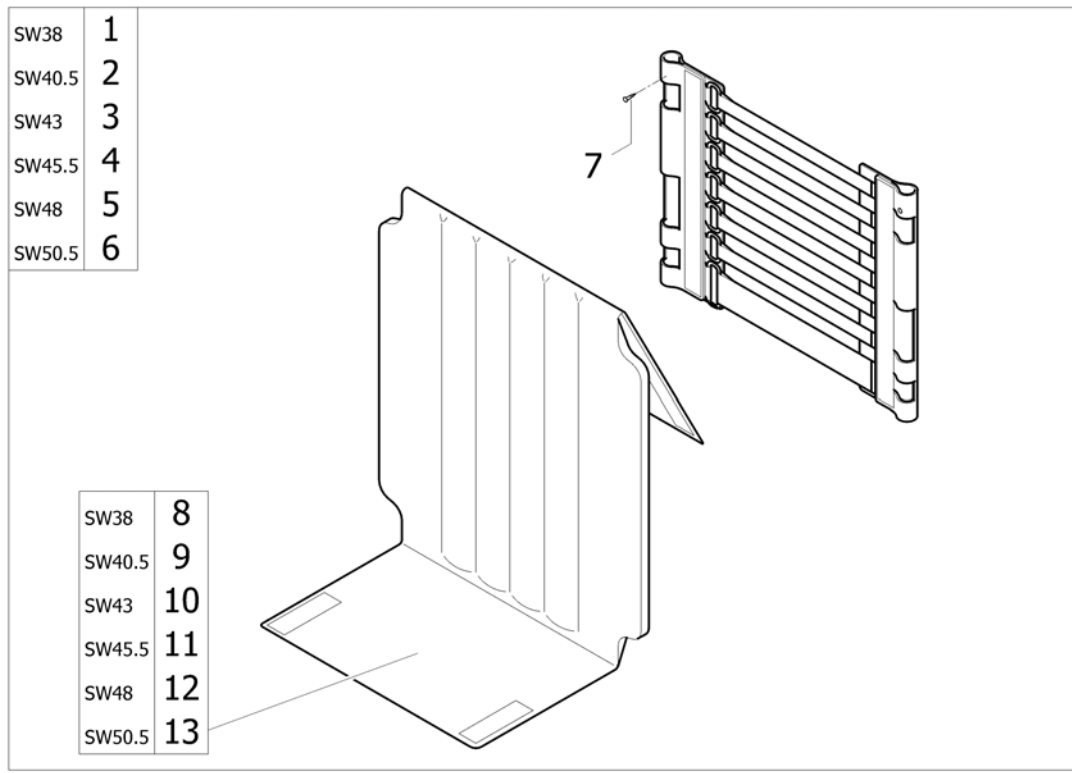


Pos.	Part number	Description	Valid from → until	Qty
1	1449965	Adjustable Back Upholstery W 38Cm X H 43Cm (Fixed)		1
2	1449966	Adjustable Back Upholstery W 40,5Cm X H 43Cm (Fixed)		1
3	1449967	Adjustable Back Upholstery W 43Cm X H 43Cm (Fixed)		1
4	1449968	Adjustable Back Upholstery W 45,5Cm X H 43Cm (Fixed)		1
5	1449969	Adjustable Back Upholstery W 48Cm X H 43Cm (Fixed)		1
6	1449970	Adjustable Back Upholstery W 50,5Cm X H 43Cm (Fixed)		1
7	0029-40005	Nickel-plated Ø4,8x16 FBZ sheet-metal screw		2
8	5106739-TR18	Cover, Backrest 38, Jemima		1
9	5106740-TR18	Cover, Backrest 40.5, Jemima		1
10	5106741-TR18	Cover, Backrest 43, Jemima		1
11	5106742-TR18	Cover, Backrest 45.5, Jemima		1
12	1435523	Cover, Backrest 48, Jemima		1
13	5106743-TR18	Cover, Backrest 50.5, Jemima		1

Comfort Upholstery

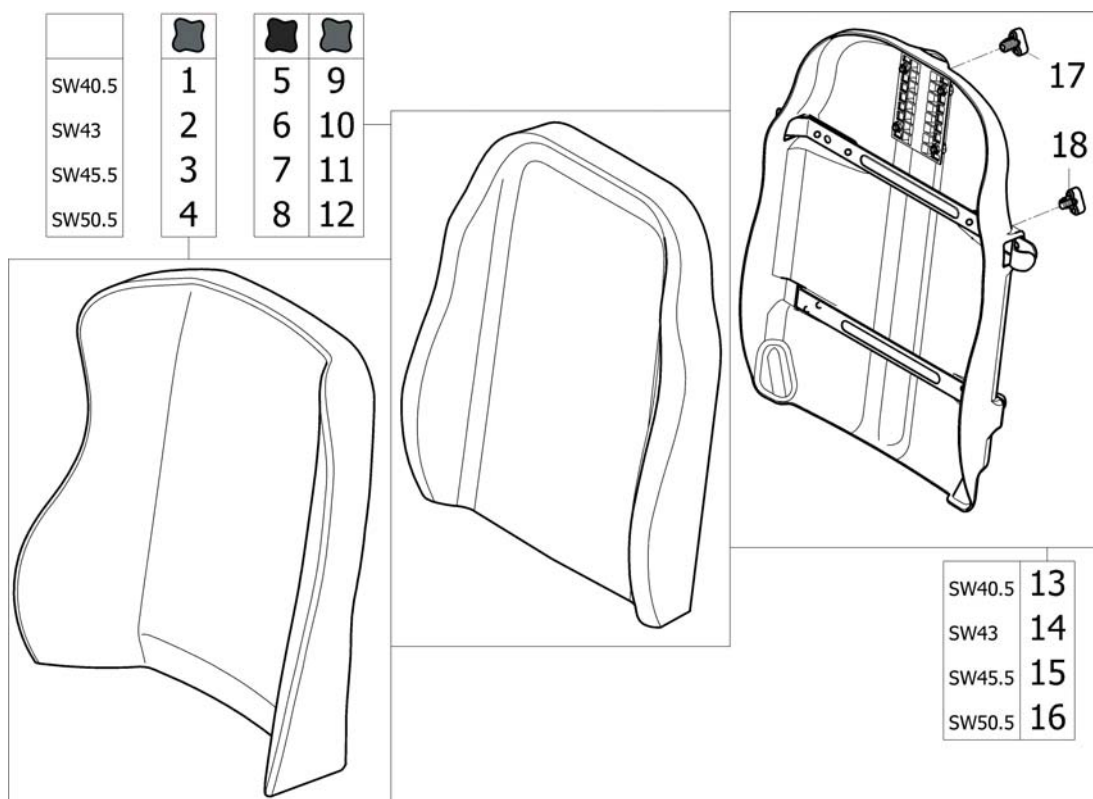

ISPS1541874


Pos.	Part number	Description	Valid from → until	Qty
1	1440951	Passad 2 Comfort Backrest Cushion (Dartex) W 40,5Cm - Xxs		1
2	1429271	Cushion, Backrest Passad2 440X550, Dartex Grey		1
3	1424647	Cushion, Backrest Passad2 470X550, Dartex Grey		1
4	1424649	Cushion, Backrest Passad2 520X550, Dartex Grey		1
5	1431516	Laguna Complet Backrest Cushion (Black Vinyl) W 40,5Cm		1
6	1420088	Laguna Complet Backrest Cushion (Black Vinyl) W 43Cm		1
7	1420089	Laguna Complet Backrest Cushion (Black Vinyl) W 45,5Cm		1
8	1420090	Laguna Complet Backrest Cushion (Black Vinyl) W 50,5Cm		1
9	1439937	CUSHION, BACKREST "LAGUNA" 340X550, DARTEX GREY		1
10	1427118	Cushion, Backrest Laguna 372X550, Dartex Grey		1
11	1427119	CUSHION,BACKREST LAGUNA 402X550,DARTEX GREY		1
12	1427120	CUSHION,BACKREST LAGUNA 452X550,TR23		1
13	1437532	Laguna Backrest Shell W 40,5Cm		1
14	1437534	Laguna Backrest Shell W 43Cm		1
15	1437536	LAGUNA BACKREST SHELL W 45,5CM		1
16	1437538	Laguna Backrest Shell W 50,5Cm		1
17	1444116	LOCKING BUTTON M8		1
18	1440943	LOCKING BUTTON M8		1

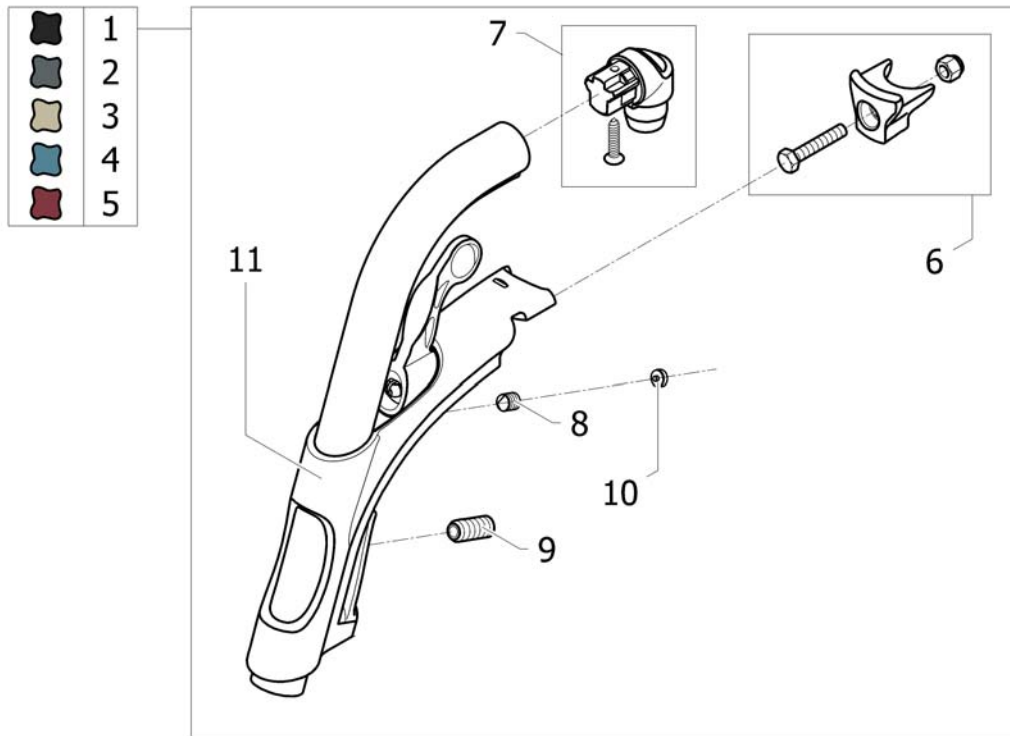Hanger fixed 80°

189133342

Pos.	Part number	Description	Valid from → until	Qty
1	1428622-0609	Hanger 80° Anthracite		2
2	1428622-0615	Footrest hanger 80° Slate Grey		2
3	1428622-0616	Footrest hanger 80° Ice Brown		2
4	1428622-0617	Footrest hanger 80° Ocean Blue		2
5	1428622-0607	Footrest Hanger 80° Dark Red		2
6	1430427	Kit, Block A3		2
7	1430426	Attachment Tap, Hanger		2
8	1424572	Stop Screw S6Ss M10x12 Din 914 Din 914/Iso 4027		2
9	1424571	Springscrew		2
10	1434070	Key cap Key		2
11	1434362	Locking House Cpl, Hanger 80 Deg		2


Legrest 90°, fixed

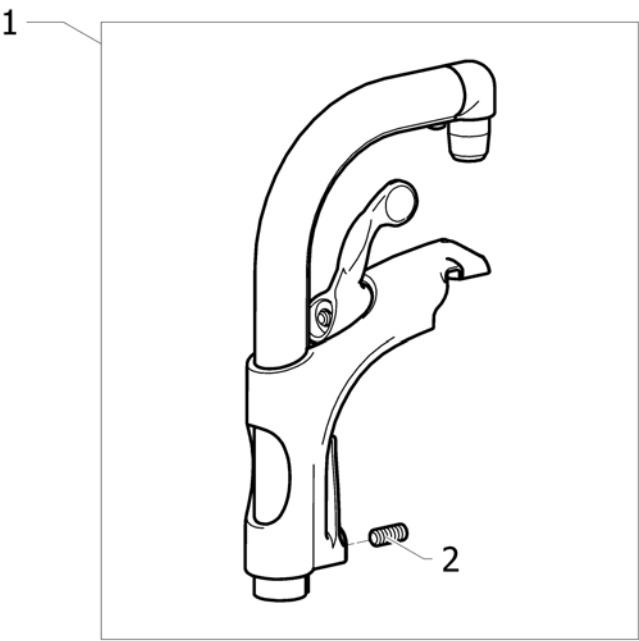

15951308038

Pos.	Part number	Description	Valid from → until	Qty
1	1493056-0509	HANGER FIXED 90		2
2	1424571	Springscrew		2


Assembly Legrest, Fixed + Ajustable Footrest

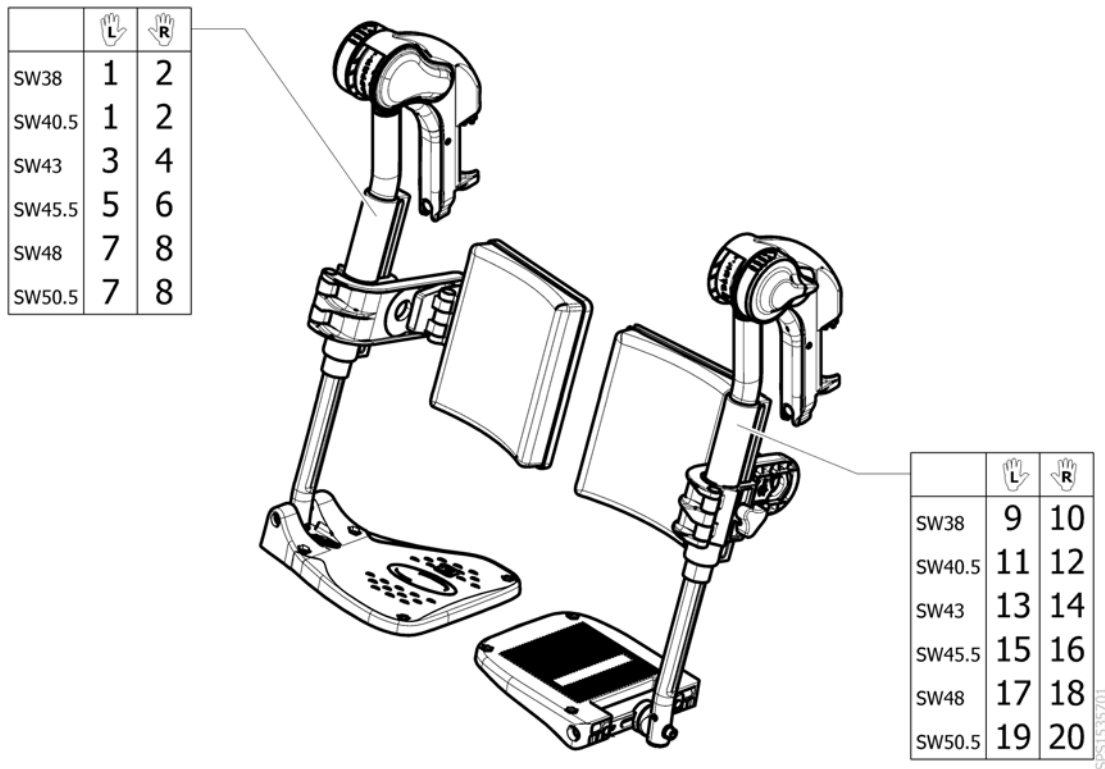

ISPS1335/01

Pos.	Part number	Description	Valid from → until	Qty
1	1432974	Lh Elevating Legrest + Fixed Footplate W 38/40,5Cm		1
2	1432973	Rh Elevating Legrest + Fixed Footplate W 38/40,5Cm		1
3	1432976	Lh Elevating Legrest + Fixed Footplate W 43Cm		1
4	1432975	Rh Elevating Legrest + Fixed Footplate W 43Cm		1
5	1432978	Lh Elevating Legrest + Fixed Footplate W 45,5Cm		1
6	1432977	Rh Elevating Legrest + Fixed Footplate W 45,5Cm		1
7	1432980	Lh Elevating Legrest + Fixed Footplate W 48/50,5Cm		1
8	1432979	Rh Elevating Legrest + Fixed Footplate W 48/50,5Cm		1
9	1432982	Lh Elevating Legrest + Adjustable Footplate W 38Cm		1
10	1432981	Rh Elevating Legrest + Adjustable Footplate W 38Cm		1
11	1432984	Lh Elevating Legrest + Adjustable Footplate W 40,5Cm		1
12	1432983	Rh Elevating Legrest + Adjustable Footplate W 40,5Cm		1
13	1432986	Lh Elevating Legrest + Adjustable Footplate W 43Cm		1
14	1432985	Rh Elevating Legrest + Adjustable Footplate W 43Cm		1
15	1432948	Lh Elevating Legrest + Adjustable Footplate W 45,5Cm		1
16	1432947	Rh Elevating Legrest + Adjustable Footplate W 45,5Cm		1
17	1432964	Lh Elevating Legrest + Adjustable Footplate W 48Cm		1
18	1432949	Rh Elevating Legrest + Adjustable Footplate W 48Cm		1
19	1432971	Lh Elevating Legrest + Adjustable Footplate W 50,5Cm		1
20	1432965	Rh Elevating Legrest + Adjustable Footplate W 50,5Cm		1


Adjustable Legrest

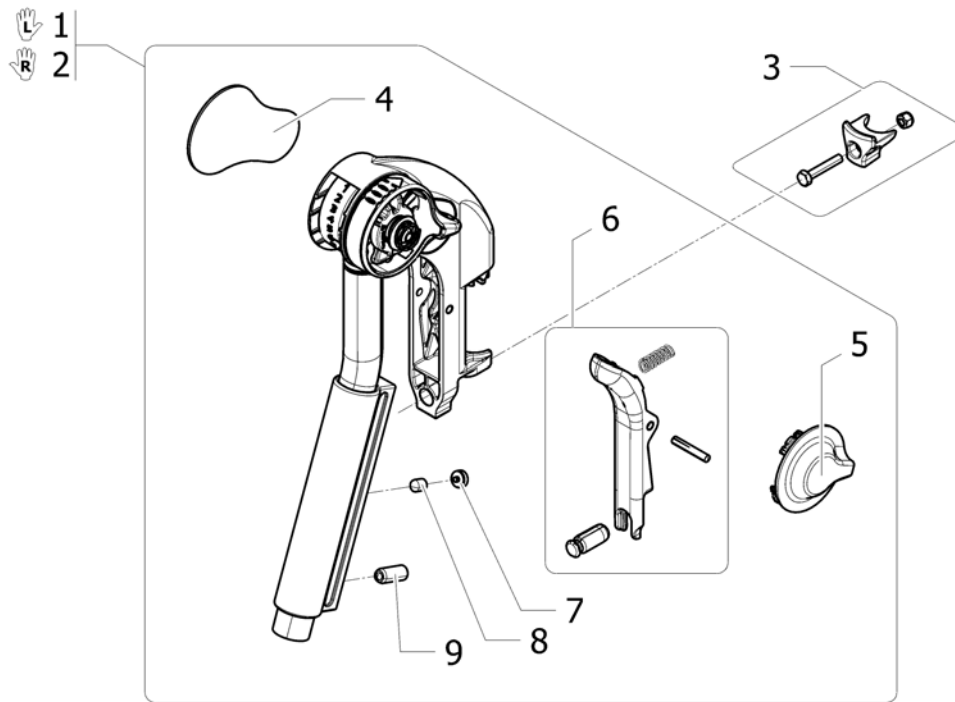

19P5133382


Pos.	Part number	Description	Valid from → until	Qty
1	1508945	Hanger, Cpl Left		1
2	1508948	Hanger, Cpl Right		1
3	1430427	Kit, Block A3		2
4	1428336	Knee Pad		2
5	1508943	Cap, Alu2008 Legrest		2
6	1430429	Legrest Trigger Kit		2
7	1434070	Key cap Key		2
8	1424572	Stop Screw S6Ss M10x12 Din 914 Din 914/Iso 4027		2
9	1424571	Springscrew		2

Calf Pad

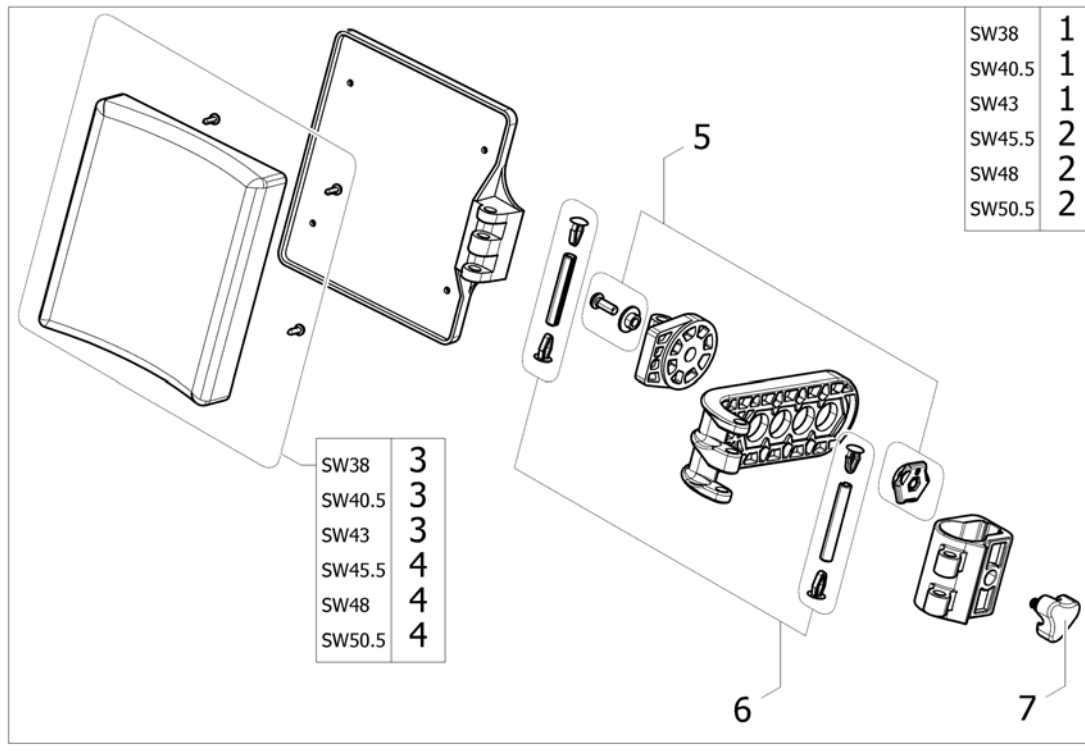


Pos.	Part number	Description	Valid from → until	Qty
1	1450932	Complete Polyurethane calf pad (cushion with support) SW38/40.5/43		2
2	1450933	Complete Polyurethane calf pad (cushion with support) SW45.5/48/50.5		2
3	1430430	Polyurethane calf pad cushion with fixation SW38/40.5/43		2
4	1430431	Polyurethane calf pad cushion with fixation SW45.5/48/50.5		2
5	1440132	Kit fixation angle adjustable calf pad		2
6	1430432	Kit articulations pins calf pad		2
7	1440943	LOCKING BUTTON M8		2

Amputee Legrest

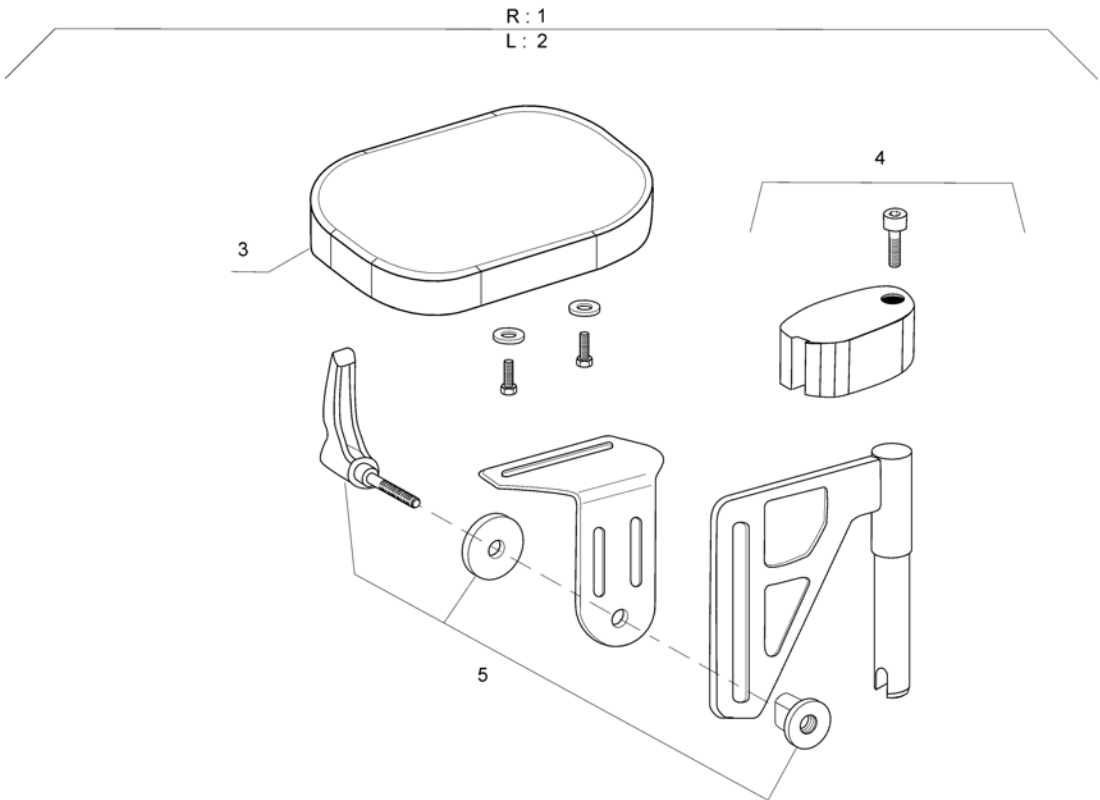


Pos.	Part number	Description	Valid from → until	Qty
1	1442990	LEGREST, AMPUTEE, RH		1
2	1442991	LEGREST, AMPUTEE, LH		1
3	5221090-TR03	Amputee Support Cushion		2
4	1446556	Amputee Support Stop + Fixation		2
5	1446557	Amputee Support Slide Fixation Kit		2

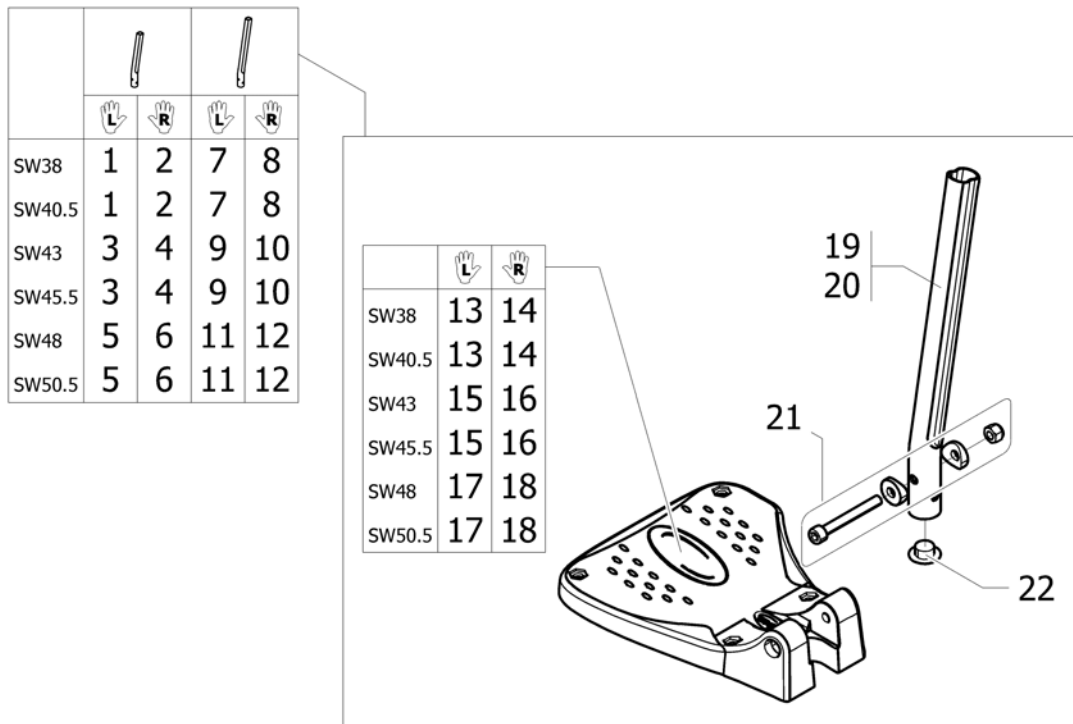Footrest

1895133043


Pos.	Part number	Description	Valid from → until	Qty
1	1530620	Lh Standard Footrest SW 38Cm/40,5Cm - Short Lower Tube		1
2	1530614	Rh Standard Footrest Sw 38Cm/40,5Cm - Short Lower Tube		1
3	1530622	Lh Standard Footrest SW 43Cm/45,5Cm - Short Lower Tube		1
4	1530616	Rh Standard Footrest Sw 43Cm/45,5Cm - Short Lower Tube		1
5	1530623	Lh Standard Footrest SW 43Cm/45,5Cm - Short Lower Tube		1
6	1530617	Rh Standard Footrest Sw 48Cm/50,5Cm - Short Lower Tube		1
7	1530633	Lh Standard Footrest SW 38Cm/40,5Cm - Long Lower Tube		1
8	1530630	Rh Standard Footrest Sw 38Cm/40,5Cm - Long Lower Tube		1
9	1530634	Lh Standard Footrest SW 43Cm/45,5Cm - Long Lower Tube		1
10	1530631	Rh Standard Footrest Sw 43Cm/45,5Cm - Long Lower Tube		1
11	1530635	Lh Standard Footrest SW 48Cm/50,5Cm - Long Lower Tube		1
12	1530632	Rh Standard Footrest Sw 48Cm/50,5Cm - Long Lower Tube		1
13	1529643	Fixed Footplate Ng Left 174mm (Sw38-40.5)		1
14	1529642	Fixed Footplate Ng Right 174mm (Sw38-40.5)		1
15	1529645	Fixed Footplate Ng Left 199mm (Sw43-45.5)		1
16	1529644	Fixed Footplate Ng Right 199mm (Sw43-45.5)		1
17	1529647	Fixed Footplate Ng Left 224mm (Sw48-50.5)		1
18	1529646	Fixed Footplate Ng Right 224mm (Sw48-50.5)		1
19	1434561-0010	Footrest Tube, Short		2
20	1434563-0010	Footrest Tube, Long		2
21	1430433	Screw Kit, Footrest Fixed		2
22	1424576	Plug Ikpt 2017S		2

Footrest with shorter support

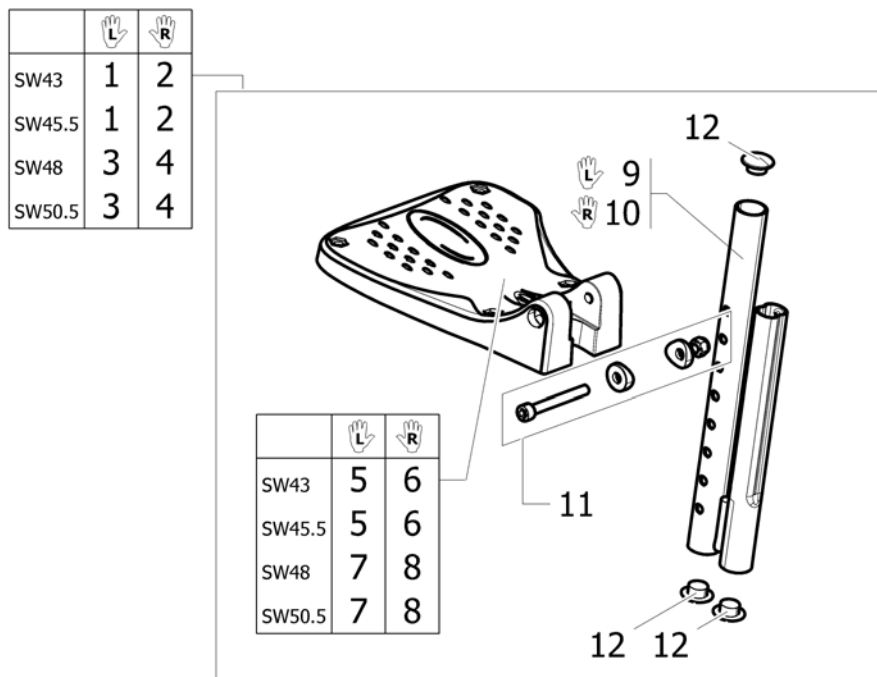


1895133711

Pos.	Part number	Description	Valid from → until	Qty
1	1532070	Fixed footrest with shorter support SW43-45.5 Left		1
2	1532066	Fixed footrest with shorter support SW43-45.5 Right		1
3	1532072	Fixed footrest with shorter support SW48-50.5 Left		1
4	1532068	Fixed footrest with shorter support SW48-50.5 Right		1
5	1529643	Fixed Footplate Ng Left 174mm (Sw38-40.5) <i>Replace 1428340</i>		1
6	1529642	Fixed Footplate Ng Right 174mm (Sw38-40.5) <i>Replace 1428341</i>		1
7	1529645	Fixed Footplate Ng Left 199mm (Sw43-45.5) <i>Replace 1424574</i>		1
8	1529644	Fixed Footplate Ng Right 199mm (Sw43-45.5) <i>Replace 1424575</i>		1
9	1531202-0699	Shorter footrest support Left		1
10	1531203-0699	Shorter footrest support Right		1
11	1430433	Screw Kit, Footrest Fixed		2
12	1424576	Plug Ikpt 2017S		6

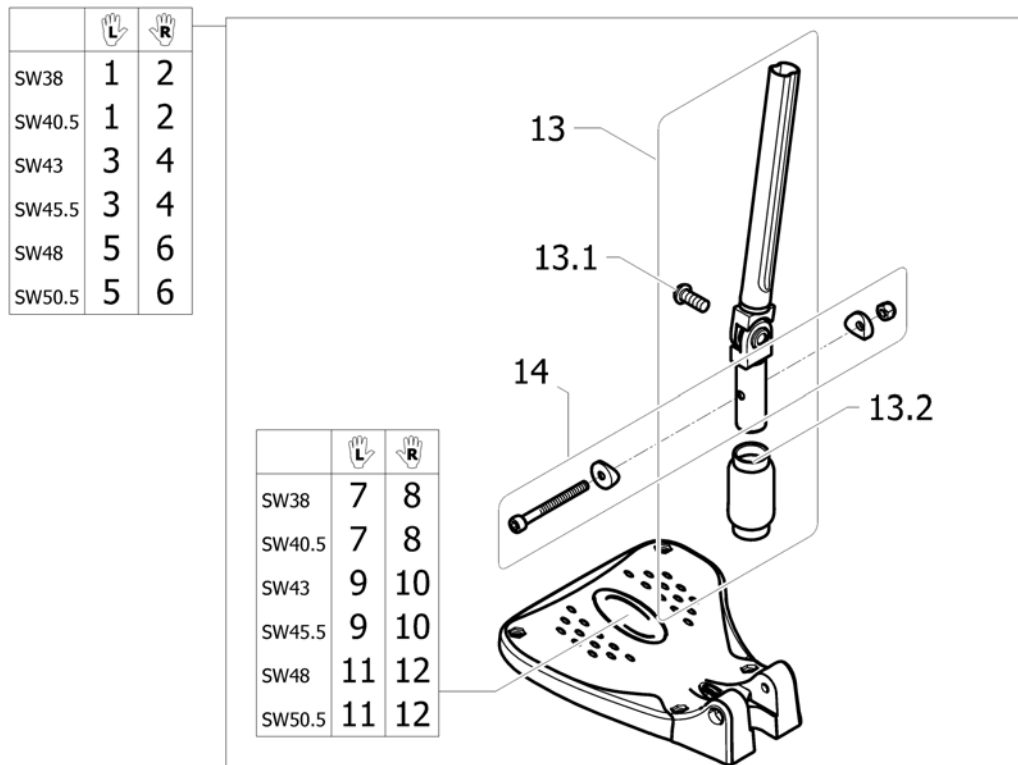Angle adjustable footrest

Pos.	Part number	Description	Valid from → until	Qty
1	1530640	Lh Adjustable Footrest SW 38-40,5Cm	→ 20100921	1
2	1530637	Rh Adjustable Footrest SW 38-40,5Cm	→ 20100921	1
3	1530641	Lh Adjustable Footrest SW 43-45,5Cm	→ 20100921	1
4	1530638	Rh Adjustable Footrest SW 43-45,5Cm	→ 20100921	1
5	1530642	Lh Adjustable Footrest SW 48-50,5Cm	→ 20100921	1
6	1530639	Rh Adjustable Footrest SW 48-50,5Cm	→ 20100921	1
7	1529643	Fixed Footplate Ng Left 174mm (Sw38-40.5)		1
8	1529642	Fixed Footplate Ng Right 174mm (Sw38-40.5)		1
9	1529645	Fixed Footplate Ng Left 199mm (Sw43-45.5)		1
10	1529644	Fixed Footplate Ng Right 199mm (Sw43-45.5)		1
11	1529647	Fixed Footplate Ng Left 224mm (Sw48-50.5)		1
12	1529646	Fixed Footplate Ng Right 224mm (Sw48-50.5)		1
13	1485167	FOOTREST TUBE AA, SHORT	→ 20100921	2
13.1	0029-40046	Screw CBHC M8X20		2
13.2	1485944	Adjustable Footrest Sleeve	→ 20100921	2
14	1430433	Screw Kit, Footrest Fixed		2

Angle and depth adjustable footrest

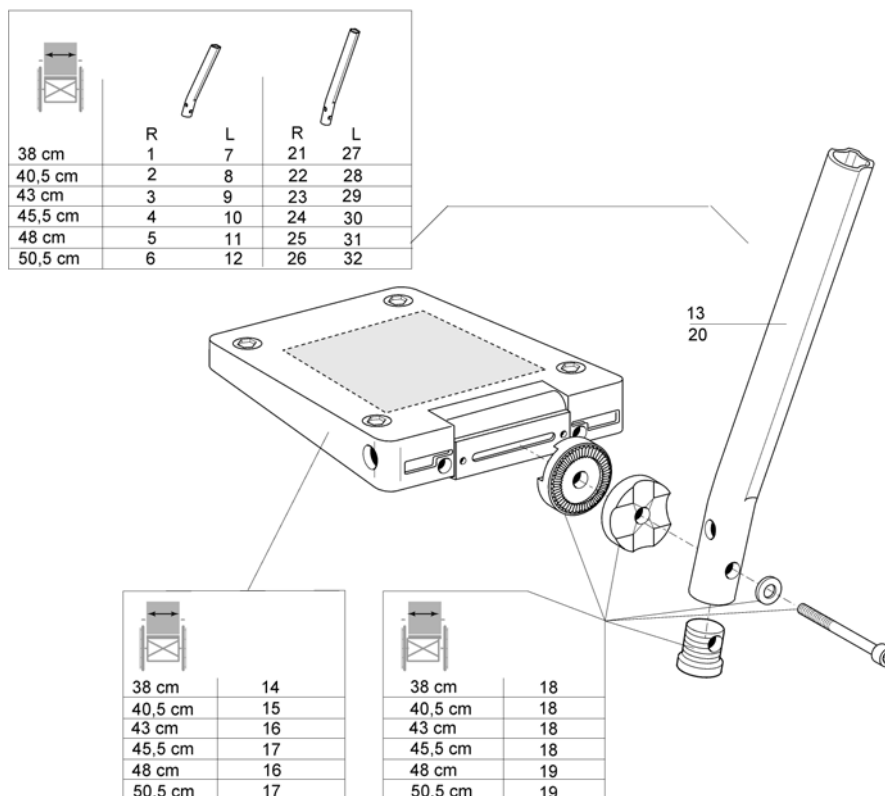


Pos.	Part number	Description	Valid from → until	Qty
1	1428651	Rh angle and depth adjustable footrest short tube SW38		1
2	1428652	Rh angle and depth adjustable footrest short tube SW40,5		1
3	1428653	Rh angle and depth adjustable footrest short tube SW43		1
4	1428654	Footplate, depth- and angleadjustable, SW46, LH, cpl., 80°		1
5	1428655	Rh angle and depth adjustable footrest short tube SW48		1
6	1428656	Rh angle and depth adjustable footrest short tube SW50,5-55,5-60,5		1
7	1428657	Lh angle and depth adjustable footrest short tube SW38		1
8	1428658	Lh angle and depth adjustable footrest short tube SW40,5		1
9	1428659	Lh angle and depth adjustable footrest short tube SW43		1
10	1428660	Footplate, depth- and angleadjustable, SW46, LH, cpl., 80°		1
11	1428661	Lh angle and depth adjustable footrest short tube SW48		1
12	1428662	Lh angle and depth adjustable footrest short tube SW50,5-55,5-60,5		1
13	1434561-0010	Footrest Tube, Short		2
14	5106374	Footplate 14 aa, Cpl.		2
15	5106375	Footplate 16 aa, Cpl.		2
16	5106594	Footplate 17 aa, Cpl.		2
17	5106376	Footplate 18 aa, Cpl.		2
18	1430839	Screw Kit, Footrest aa		2
19	1430840	Screw Kit, Footrest aa Ext		2
20	1434563-0010	Footrest Tube, Long		2
21	1440810	Rh angle and depth adjustable footrest long tube SW38		1
22	1440811	Rh angle and depth adjustable footrest long tube SW40,5		1
23	1440812	Rh angle and depth adjustable footrest long tube SW43		1
24	1440813	Rh angle and depth adjustable footrest long tube SW45,5		1
25	1440814	Rh angle and depth adjustable footrest long tube SW48		1
26	1440815	Rh angle and depth adjustable footrest long tube SW50,5		1
27	1440816	Lh angle and depth adjustable footrest long tube SW38		1
28	1440817	Lh angle and depth adjustable footrest long tube SW40,5		1
29	1440818	Lh angle and depth adjustable footrest long tube SW43		1
30	1440819	Lh angle and depth adjustable footrest long tube SW45,5		1
31	1440820	Lh angle and depth adjustable footrest long tube SW48		1

Angle and depth adjustable footrest

Pos.	Part number	Description	Valid from → until	Qty
32	1440821	Lh angle and depth adjustable footrest long tube SW50,5		1


Angle and depth adjustable footrest with shorter support

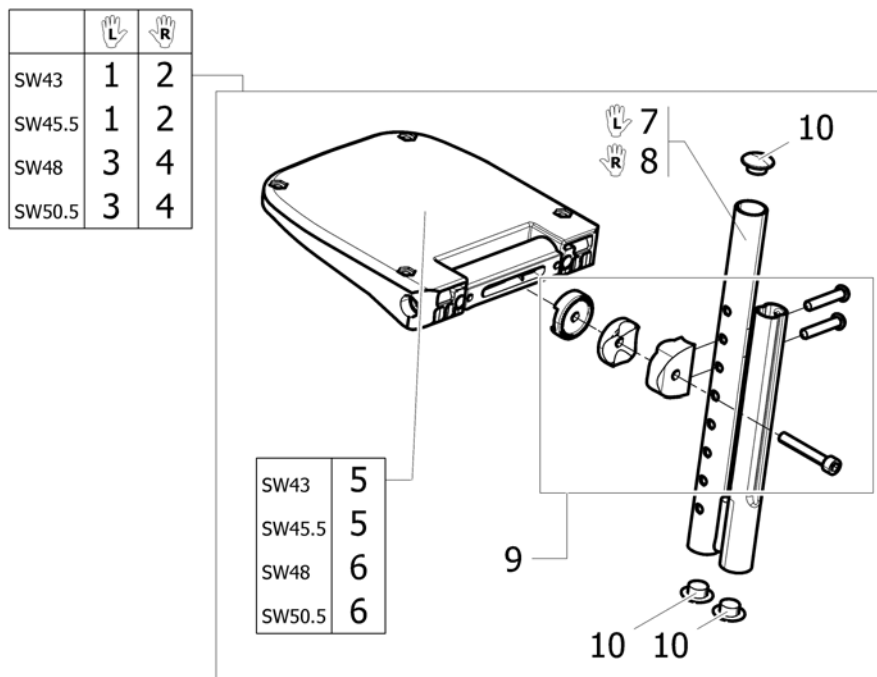

1895133713


Pos.	Part number	Description	Valid from → until	Qty
1	1532078	Angle and depth adjustable footrest with shorter support SW43-45.5 Left		1
2	1532074	Angle and depth adjustable footrest with shorter support SW43-45.5 Right		1
3	1532080	Angle and depth adjustable footrest with shorter support SW48-50.5 Left		1
4	1532076	Angle and depth adjustable footrest with shorter support SW48-50.5 Right		1
5	5106374	Footplate 14 aa, Cpl.		2
6	5106594	Footplate 17 aa, Cpl.		2
7	1531202-0699	Shorter footrest support Left		1
8	1531203-0699	Shorter footrest support Right		1
9	1533714	Screws kit for angle and depth adjustable footrest with shorter support		2
10	1424576	Plug Ikpt 2017S		6

One piece footboard

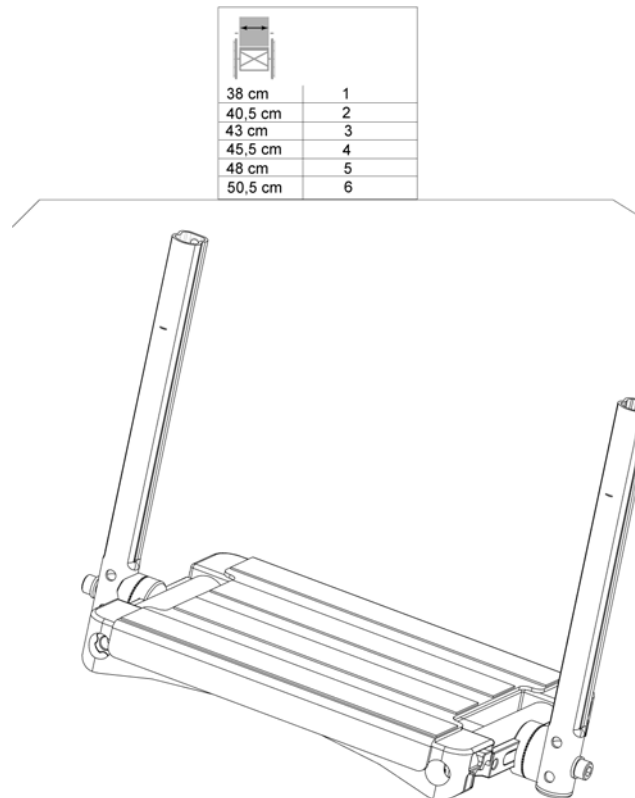

Pos.	Part number	Description	Valid from → until	Qty
1	1496945	FOOTREST OP, 38, CPL DEPTH AND A/A,SHORT, 80		1
2	1496946	FOOTREST OP, 40.5, CPL DEPTH AND A/A,SHORT, 80		1
3	1496947	FOOTREST OP, 43, CPL DEPTH AND A/A,SHORT, 80		1
4	1496948	FOOTREST OP, 45.5, CPL DEPTH AND A/A,SHORT, 80		1
5	1496949	FOOTREST OP, 48, CPL DEPTH AND A/A,SHORT, 80		1
6	1496953	FOOTREST OP, 50.5, CPL DEPTH AND A/A,SHORT, 80		1


Armrest n°1 - Removable and height adjustable

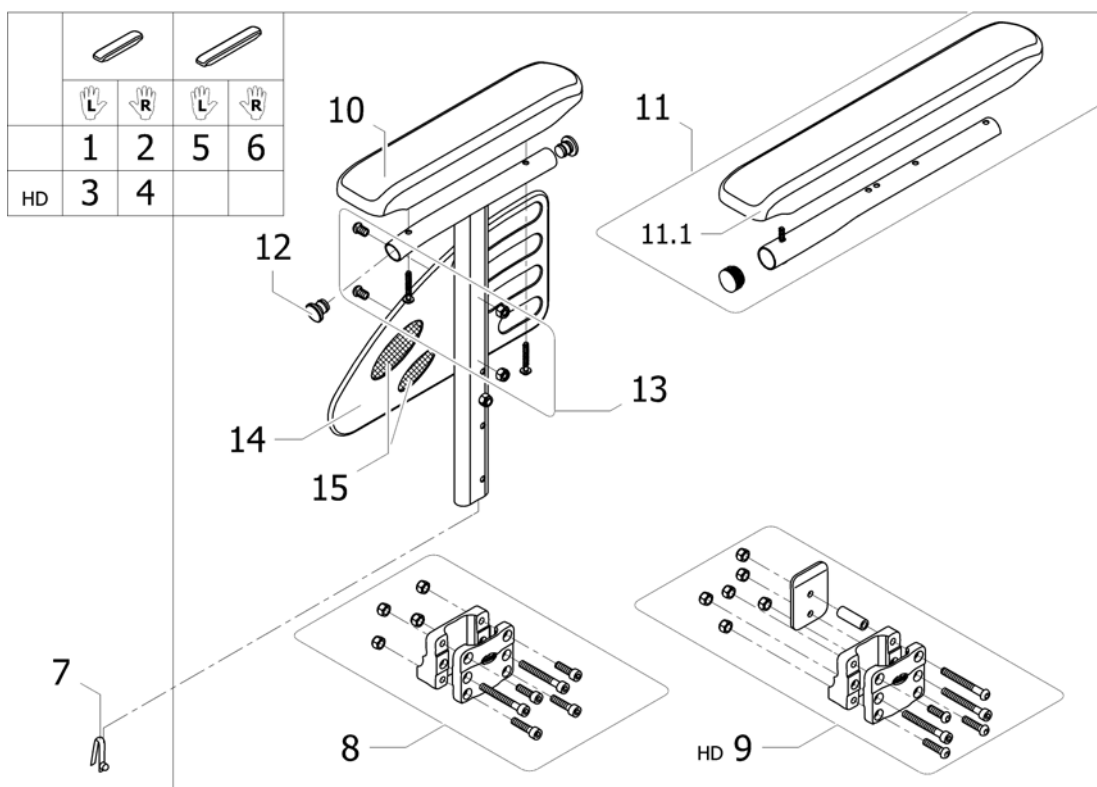

ISPS1495339

Pos.	Part number	Description	Valid from → until	Qty
1	1534086-0601	Lh Desk Armrest n°1 - Removable and height adjustable <i>Replace 1428597-0601</i>		1
2	1534085-0601	Rh Desk Armrest n°1 - Removable and height adjustable <i>Replace 1428596-0601</i>		1
3	1541935	Armrest n°1 - Removable and height adjustable short armpad Left heavy		1
4	1541934	Armrest n°1 - Removable and height adjustable short armpad Right heavy		1
5	1534088-0601	Lh Long Armrest n°1 - Removable and height adjustable <i>Replace 1428599-0601</i>		1
6	1534087-0601	Rh Long Armrest n°1 - Removable and height adjustable <i>Replace 1428598-0601</i>		1
7	0027-10003	Stud spring Ø6mm		2
8	1430421	Armrest Attachment		2
9	1532575	Central Support Armrest 1 Kit Hd		2
10	1543220	Kit armpad short 280mm + 2 screws <i>Replace 1426721</i>		2
11	1432595	Long armpad kit + tube + screw		2
11.1	1543247	Kit armpad long 380mm + 3 screws <i>Replace 1426723</i>		2
12	0021-20028	Plastic cap black Ø19x1,5 mm		2
13	1430422	Armrest n°1 screws kit		2
14	1425601	Armrest Skirtguard 1		2
15	1535085	Skirtguard sticker kit N°1		2

Swing Away & Detachable Armrest 2

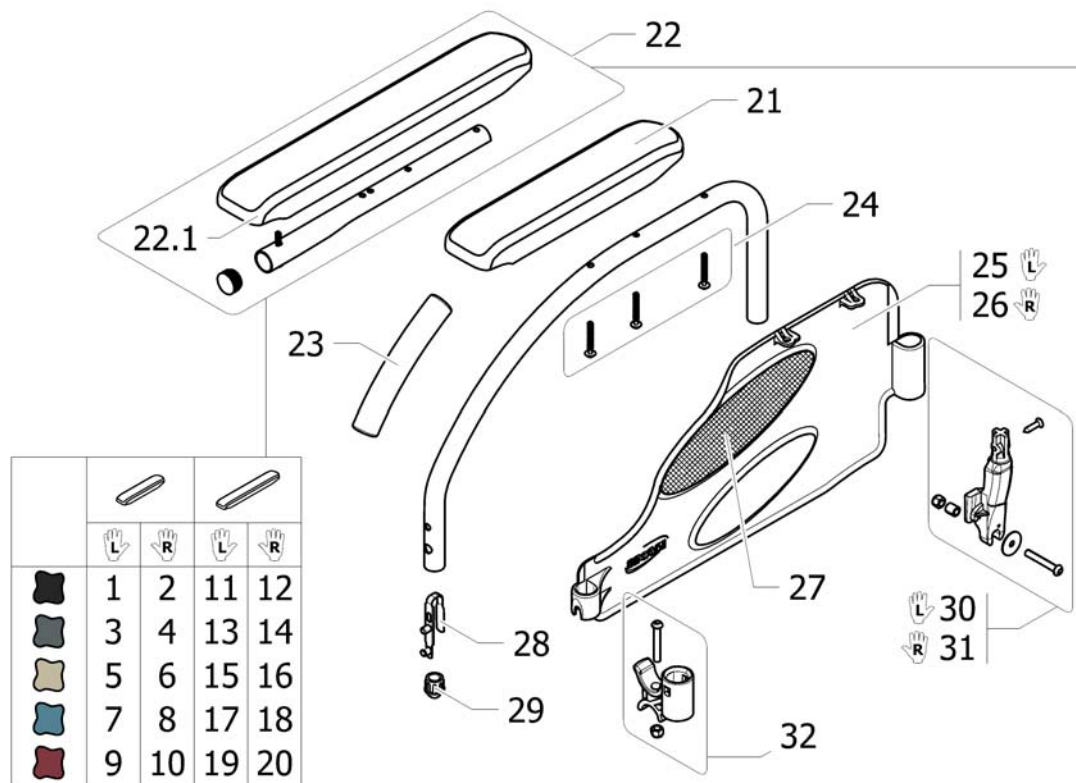


Pos.	Part number	Description	Valid from → until	Qty
1	1428605-0609	Lh Desk Armrest 2; Anthracite Black		1
2	1428604-0609	Rh Desk Armrest 2; Anthracite Black		1
3	1428605-0615	Lh Desk Armrest 2; Slate Grey		1
4	1428604-0615	Rh Desk Armrest 2; Slate Grey		1
5	1428605-0616	Lh Desk Armrest 2; Ice Brown		1
6	1428604-0616	Rh Desk Armrest 2; Ice Brown		1
7	1428605-0617	Lh Desk Armrest 2; Ocean Blue		1
8	1428604-0617	Rh Desk Armrest 2; Ocean Blue		1
9	1428605-0607	Lh Desk Armrest 2; Dark Red		1
10	1428604-0607	Rh Desk Armrest 2; Dark Red		1
11	1428607-0609	Lh Long Armrest 2; Anthracite Black		1
12	1428606-0609	Rh Long Armrest 2; Anthracite Black		1
13	1428607-0615	Lh Long Armrest 2; Slate Grey		1
14	1428606-0615	Rh Long Armrest 2; Slate Grey		1
15	1428607-0616	Lh Long Armrest 2; Ice Brown		1
16	1428606-0616	Rh Long Armrest 2; Ice Brown		1
17	1428607-0617	Lh Long Armrest 2; Ocean Blue		1
18	1428606-0617	Rh Long Armrest 2; Ocean Blue		1
19	1428607-0607	Lh Long Armrest 2; Dark Red		1
20	1428606-0607	Rh Long Armrest 2; Dark Red		1
21	1543195	Kit armpad short 280mm + 3 screws <i>Replace 1426721</i>		2
22	1432595	Long armpad kit + tube + screw		2
22.1	1543219	Kit armpad long 380mm + 4 screws <i>Replace 1426723</i>		2
23	0572-02851	Armrest handle cover		2
24	1430424	Armrest 2 Screws Kit N°2		2
25	1532799	Skirtguard Armrest 2 Lh with sticker		1
26	1532800	Skirtguard Armrest 2 Rh with sticker		1
27	1532794	Skirtguard sticker N°2		2
28	1018225	Double stud spring		2
29	0572-03405	Short conical cap		2

Swing Away & Detachable Armrest 2

Pos.	Part number	Description	Valid from → until	Qty
30	1431284	Lh Armrest Rear Support Kit		1
31	1430423	Rh Armrest Rear Support Kit		1
32	0572-05368	Crossbar and armrest support kit		2

Height Adjustable Armrest 3

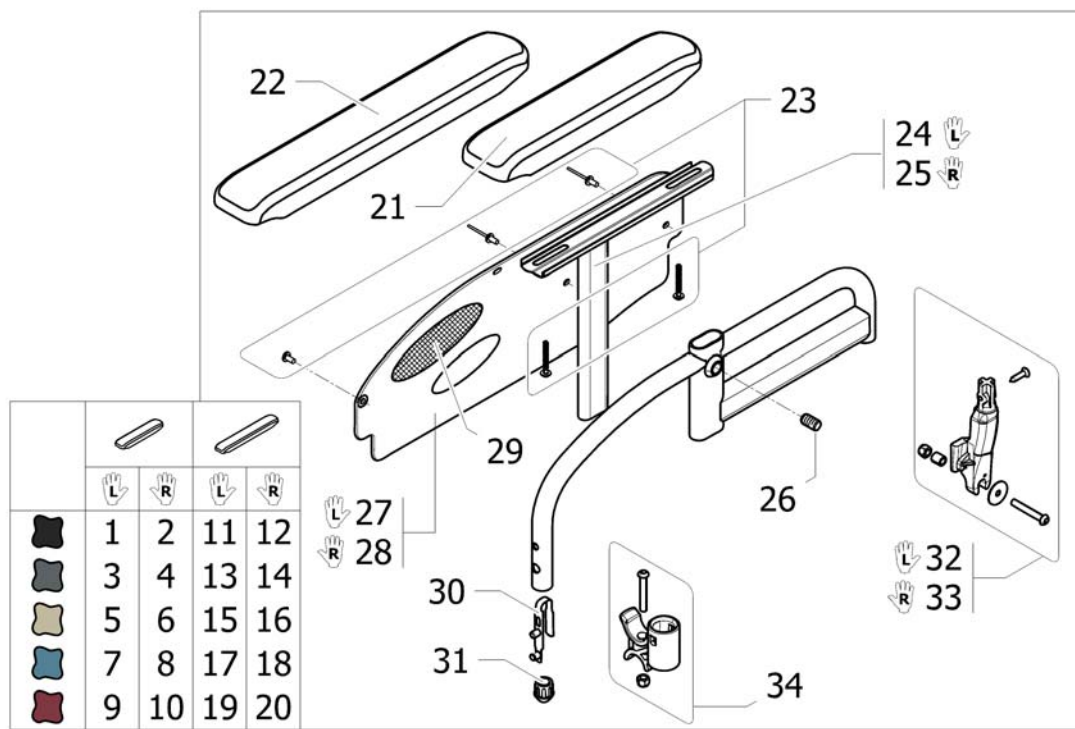


Pos.	Part number	Description	Valid from → until	Qty
1	1428613-0609	Lh Desk Armrest 3; H & D Adjustable; Anthracite Black		1
2	1428612-0609	Rh Desk Armrest 3; H & D Adjustable; Anthracite Black		1
3	1428613-0615	Lh Desk Armrest 3; H & D Adjustable; Slate Grey		1
4	1428612-0615	Rh Desk Armrest 3; H & D Adjustable; Slate Grey		1
5	1428613-0616	Lh Desk Armrest 3; H & D Adjustable; Ice Brown		1
6	1428612-0616	Rh Desk Armrest 3; H & D Adjustable; Ice Brown		1
7	1428613-0617	Lh Desk Armrest 3; H & D Adjustable; Ocean Blue		1
8	1428612-0617	Rh Desk Armrest 3; H & D Adjustable; Ocean Blue		1
9	1428613-0607	Lh Desk Armrest 3; H & D Adjustable; Dark Red		1
10	1428612-0607	Rh Desk Armrest 3; H & D Adjustable; Dark Red		1
11	1428615-0609	Lh Long Armrest 3; H & D Adjustable; Anthracite Black		1
12	1428614-0609	Rh Long Armrest 3; H & D Adjustable; Anthracite Black		1
13	1428615-0615	Lh Long Armrest 3; H & D Adjustable; Slate Grey		1
14	1428614-0615	Rh Long Armrest 3; H & D Adjustable; Slate Grey		1
15	1428615-0616	Lh Long Armrest 3; H & D Adjustable; Ice Brown		1
16	1428614-0616	Rh Long Armrest 3; H & D Adjustable; Ice Brown		1
17	1428615-0617	Lh Long Armrest 3; H & D Adjustable; Ocean Blue		1
18	1428614-0617	Rh Long Armrest 3; H & D Adjustable; Ocean Blue		1
19	1428615-0607	Lh Long Armrest 3; H & D Adjustable; Dark Red		1
20	1428614-0607	Rh Long Armrest 3; H & D Adjustable; Dark Red		1
21	1434676	Armpad, Short		2
22	1434677	Long Armpad Flat Support		2
23	1535946	Armrest 3 Screws Kit		2
24	1449638-0010	Armpad Support Lh		1
25	1449637-0010	Adjustable Mounting Tube Lh		1
26	5106533	Locking Screw M12x10		2
27	1532801	Skirtguard Armrest 3 Lh with sticker		1
28	1532802	Skirtguard Armrest 3 Rh with sticker		1
29	1532795	Skirtguard sticker N°3		2
30	1018225	Double stud spring		2
31	0572-03405	Short conical cap		2

Height Adjustable Armrest 3

Pos.	Part number	Description	Valid from → until	Qty
32	1431284	Lh Armrest Rear Support Kit		1
33	1430423	Rh Armrest Rear Support Kit		1
34	0572-05368	Crossbar and armrest support kit		2


Height Adjustable Comfort Armrest

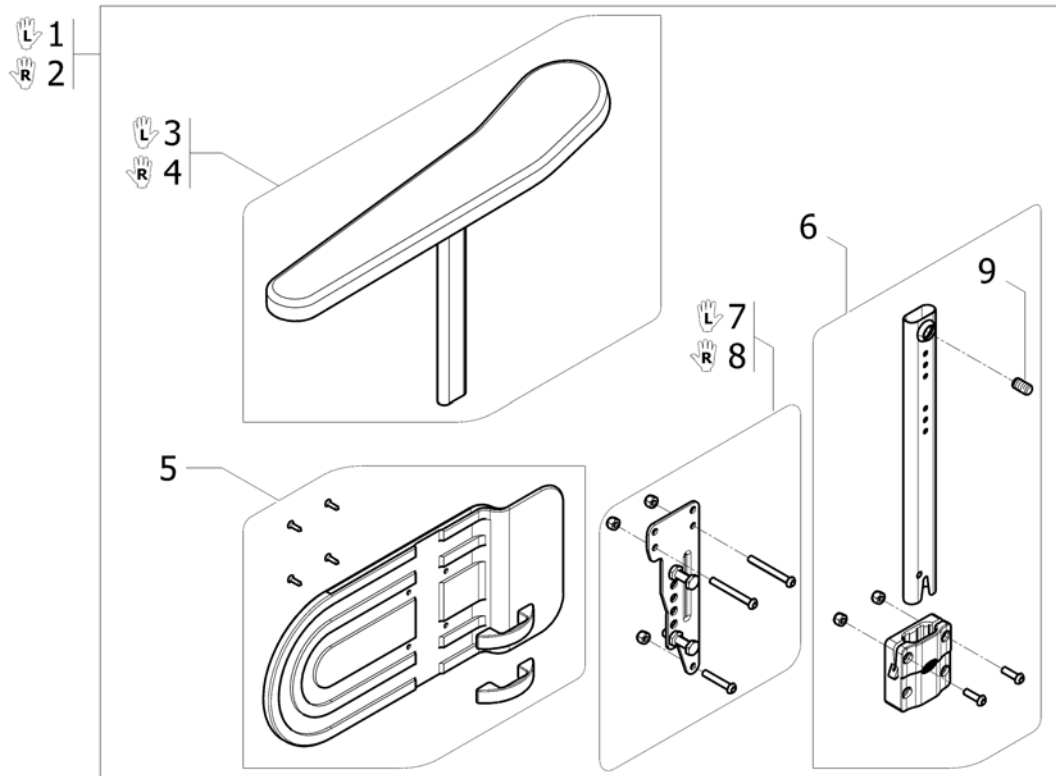

1995149342

Pos.	Part number	Description	Valid from → until	Qty
1	1440894	Lh comfort armrest 4; H & D Adjustable		1
2	1440893	Rh comfort armrest 4; H & D Adjustable		1
3	1518702	ARMREST UPPER PART, HIGHT ADJ. LEFT		1
4	1518701	ARMREST UPPER PART, HIGHT ADJ. RIGHT <i>Replace 1435832</i>		1
5	1430425	Skirtguard Armrest 4 Fixation Kit		2
6	1440952	Central Support Armrest Kit Comfort		2
7	1442896	Lh Armrest Support Plate		1
8	1440953	Rh Armrest Support Plate		1
9	5106533	Locking Screw M12x10		2


Height Adjustable Armrest 4

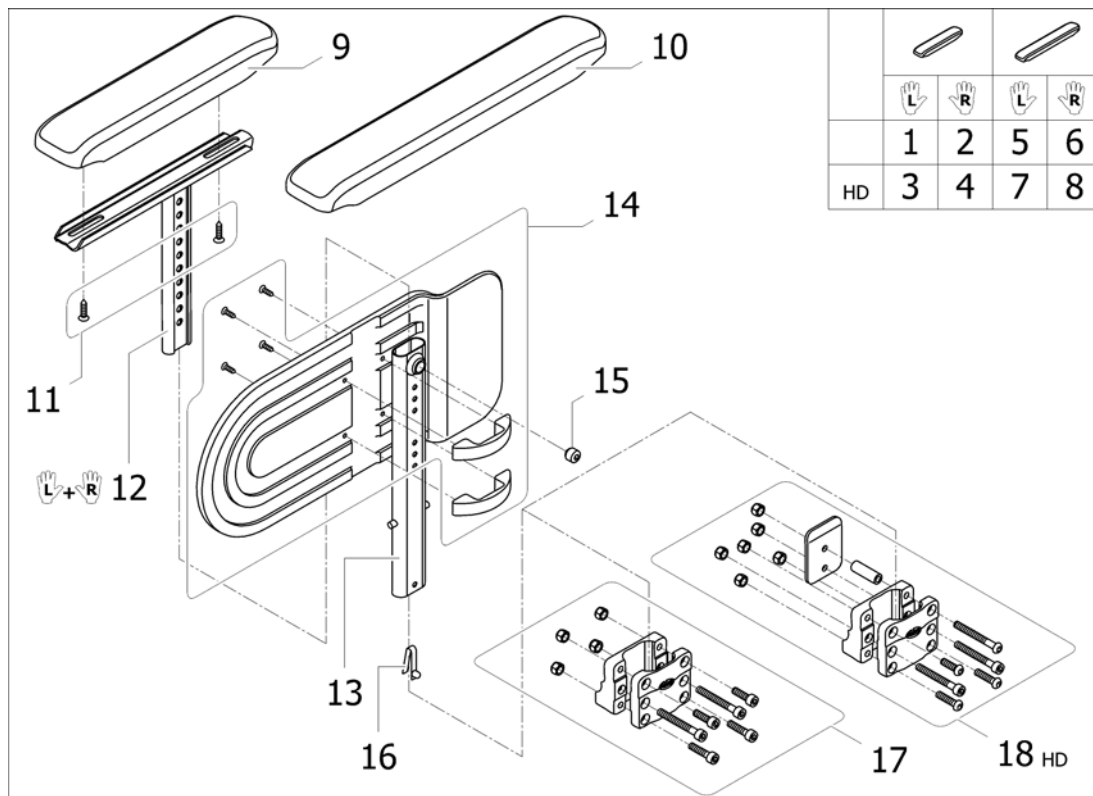

ISPS14963381


Pos.	Part number	Description	Valid from → until	Qty
1	1449988	Adjustable Armrest 4, Short Pad Lh		1
2	1449987	Adjustable Armrest 4, Short Pad Rh		1
3	1487438	Adjustable Armrest 4, Short Pad Lh Hd		1
4	1487437	Adjustable Armrest 4, Short Pad Rh Hd		1
5	1450016	Adjustable Armrest 4, Long Pad Lh		1
6	1450015	Adjustable Armrest 4, Long Pad Rh		1
7	1487440	Adjustable Armrest 4, Long Pad Lh Hd		1
8	1487439	Adjustable Armrest 4, Long Pad Rh Hd		1
9	1434676	Ampad, Short		2
10	1434677	Long Ampad Flat Support		2
11	0029-40532	Fbz Screw D 4,8X19		4
12	1524058	Ampad Support Pair		1
13	1443058	Lowest Arm Rest Tube Support N°4		2
14	1430425	Skirtguard Armrest 4 Fixation Kit		2
15	5106533	Locking Screw M12x10		2
16	0027-10003	Stud spring Ø6mm		2
17	1430421	Armrest Attachment		2
18	1532575	Central Support Armrest 1 Kit Hd		1

Hemi Dartex Armpad Multiposition

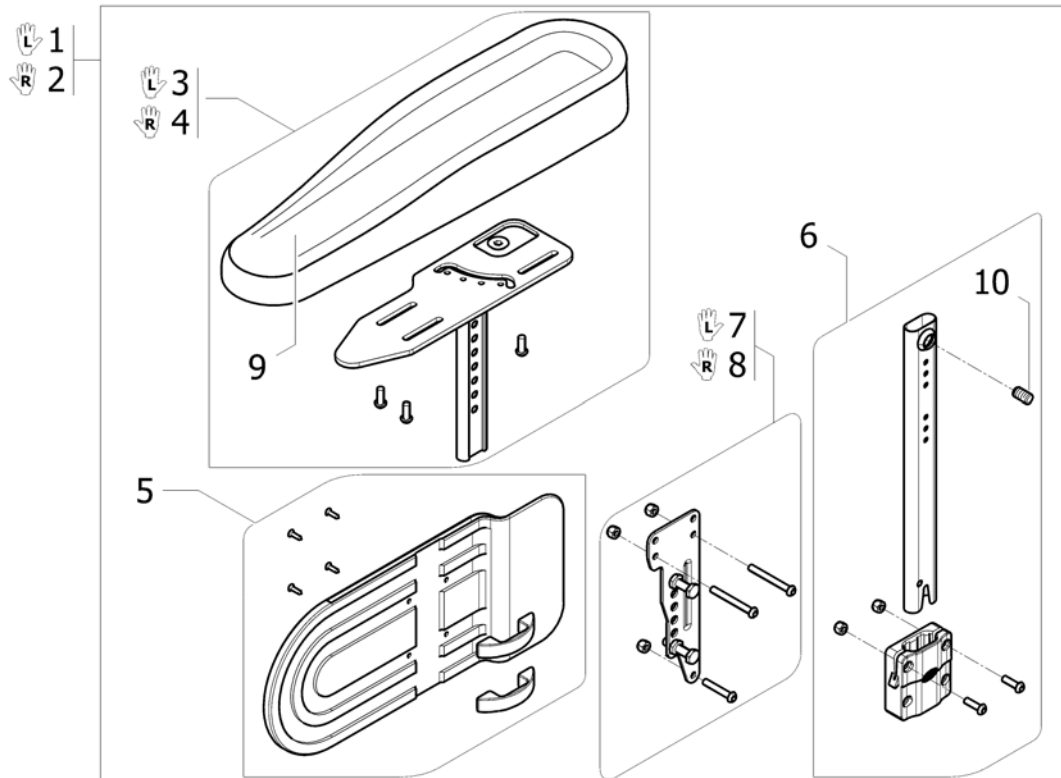


Pos.	Part number	Description	Valid from → until	Qty
1	1497039	Lh hemi armrest 4; multiposition		1
2	1497008	Rh hemi armrest 4; multiposition		1
3	1493220	ARMREST HEMI LE, AZALEA		1
4	1493219	ARMREST HEMI RI, AZALEA		1
5	1430425	Skirtguard Armrest 4 Fixation Kit		2
6	1440952	Central Support Armrest Kit Comfort		2
7	1442896	Lh Armrest Support Plate		1
8	1440953	Rh Armrest Support Plate		1
9	1493251	Armpad, Hemi Armrest, Azalea		2
10	5106533	Locking Screw M12x10		2

Mudguard Xtra

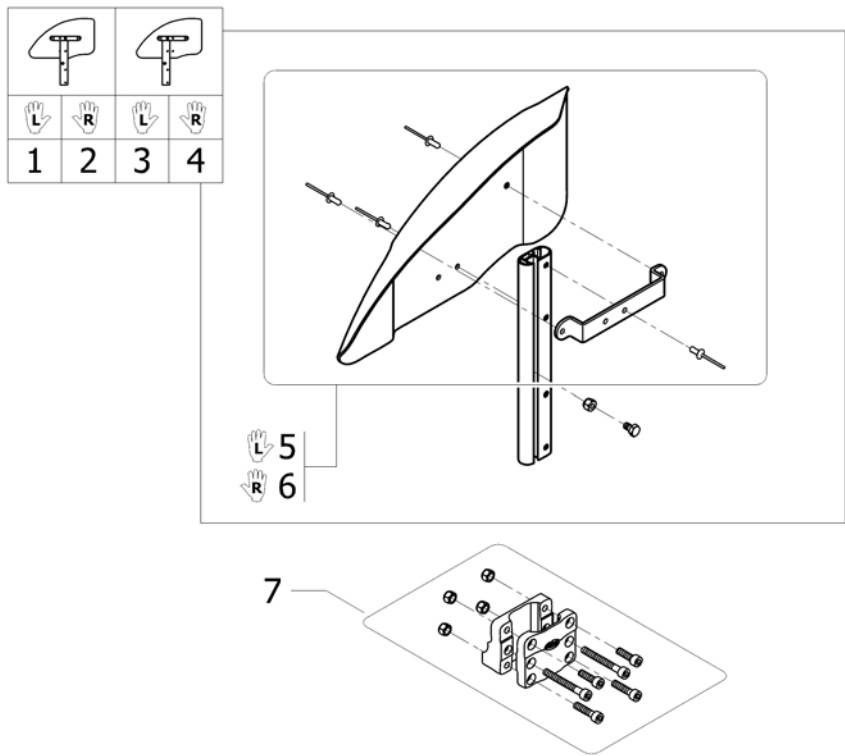

1975149533


Pos.	Part number	Description	Valid from → until	Qty
1	1486042	Lh Mudguard A3 Complete		1
2	1486041	Rh Mudguard A3 Complete		1
3	1492739	Lh Mudguard A3-A3J Complete Back Position		1
4	1492738	Rh Mudguard A3-A3J Complete Back Position		1
5	1488806	Lh Mudguard + Fixation		1
6	1488805	Rh Mudguard + Fixation		1
7	1430421	Armrest Attachment		2

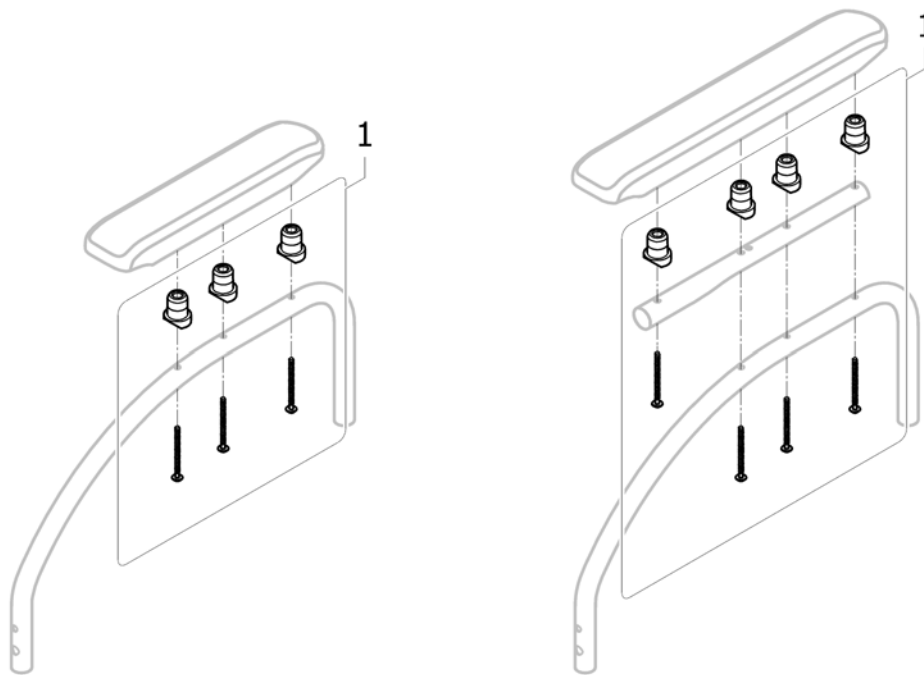Armpads raiser kit

159133360


Pos.	Part number	Description	Valid from → until	Qty
1	1543248	Pair of armpads raiser kit 2 x 4 spacers + screws <i>Replace 1443006</i>	20100610 →	1
1	1443006	Pair of armpads raiser kit 2 x 4 spacers Action 2000 <i>Replaced by 1543248</i>	→ 20100609	1

Forks

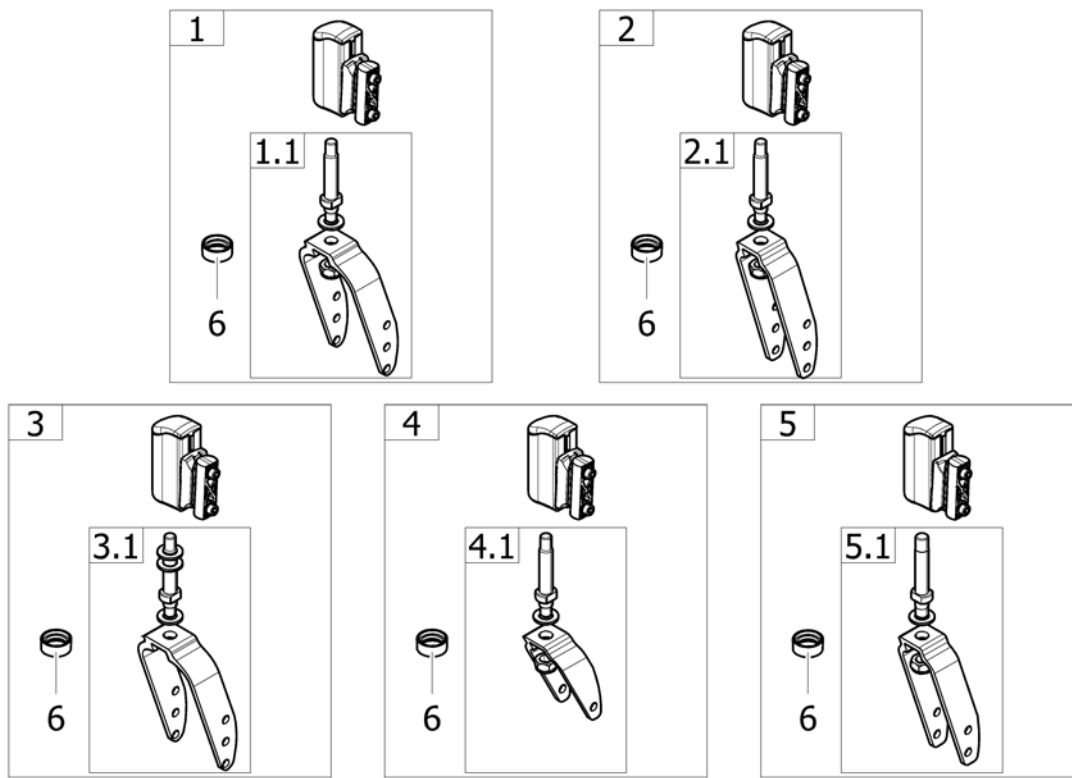


Pos.	Part number	Description	Valid from → until	Qty
1	1450025	Kit fork L60 + axle + support + fixation (Wheels 8 & 6")		2
1.1	1444802	Kit fork L60 + axle (Wheels 8 & 6")		2
2	1450026	Kit fork L36 + axle + support + fixation (Wheels 8 & 6")		2
2.1	1444803	Kit fork L36 + axle (Wheels 8 & 6")		2
3	1538177	Kit fork L60 + axle + support + fixation (Wheels 51/2)		2
3.1	1515699	Kit fork L60 + axle (Wheels 51/2)		2
4	1450027	Complete Fork Kit L36 One Position (5)		2
4.1	1451775	Fork Kit L36 (5)		2
5	1513247	Complete Fork Kit L36 <i>replace 1529518</i>		2
5.1	1513536	Fork Complete L36 A3 Junior		2
6	1529516-0618	Fork Ring "Champagne"		2

Forks supports

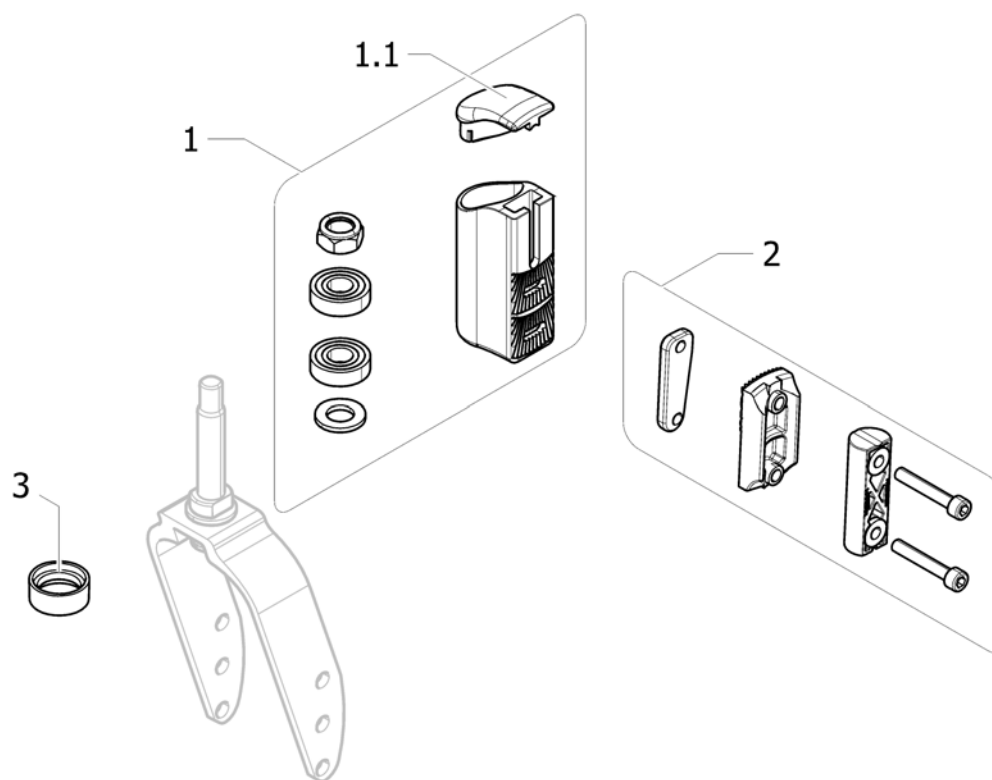

Pos.	Part number	Description	Valid from → until	Qty
1	1444804	Kit fork support		2
1.1	1449245	Outer Housing Plug		2
2	1450028	Kit fork support fixation		2
3	1529516-0618	Fork Ring "Champagne"		2


Front wheels

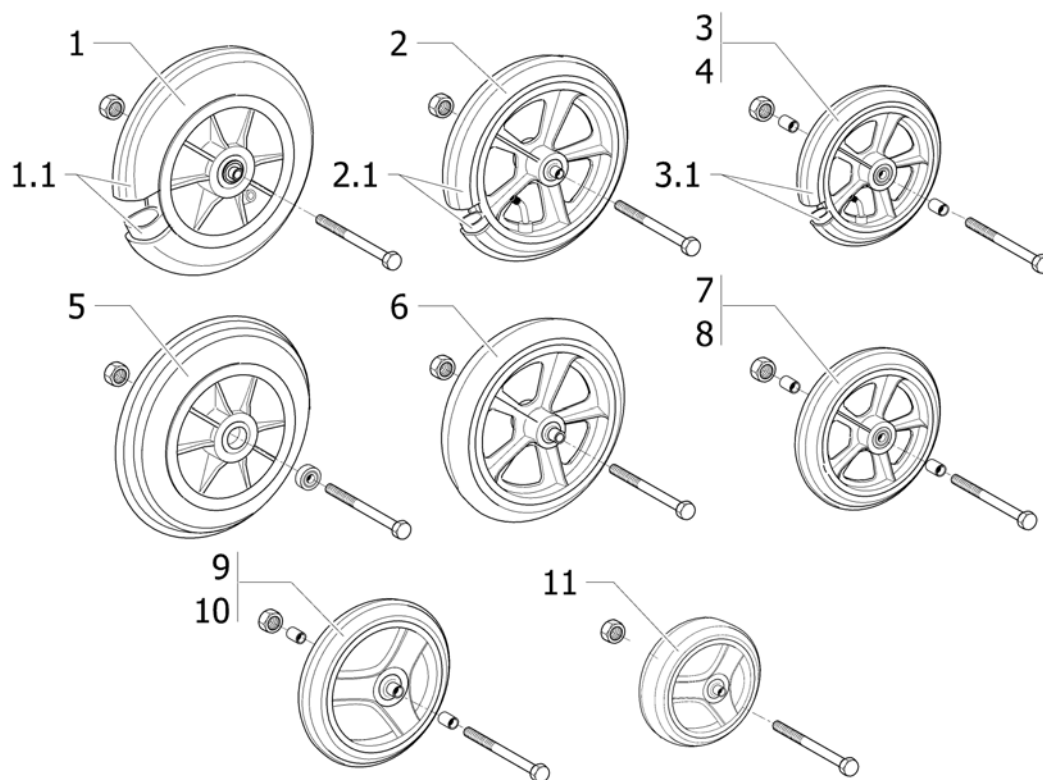

Pos.	Part number	Description	Valid from → until	Qty
1	0572-02071	8" x 2" castor pneumatic tyre complete		2
1.1	1430874	8" x 2" pneumatic tyre + inner tube		2
2	0572-01859	8" x 1.1/4" castor pneumatic tyre complete		2
2.1	1430873	8" x 1.1/4" pneumatic tyre + inner tube		2
3	1450175	Castor 150mm (6" x 1.1/4"), Pneumatic, Cpl.		2
3.1	1431275	6" x 1.1/4" pneumatic tyre + inner tube		2
4	1428894	6" x 1.1/4" castor pneumatic tyre complete 60 mm		2
5	0572-06128	8" x 2" castor solid tyre complete		2
6	0572-02013	8" x 1.1/4" castor solid tyre complete		2
7	1450174	Castor 150mm (6" x 1.1/4"), Solid, Cpl.		2
8	1428895	6" x 1.1/4" castor solid tyre complete L 60		2
9	1450622	Castor 150mm (6" x 1.1/4"), Solid Medium Hard, Cpl.		2
10	1450176	A4 6 Castor Medium Hard Type Complete L : 60 Mm		2
11	1515254	Castor 140mm (5.1/2" x 1.3/4"), Solid Medium Hard, Cpl.		2

Front wheels Compact


1

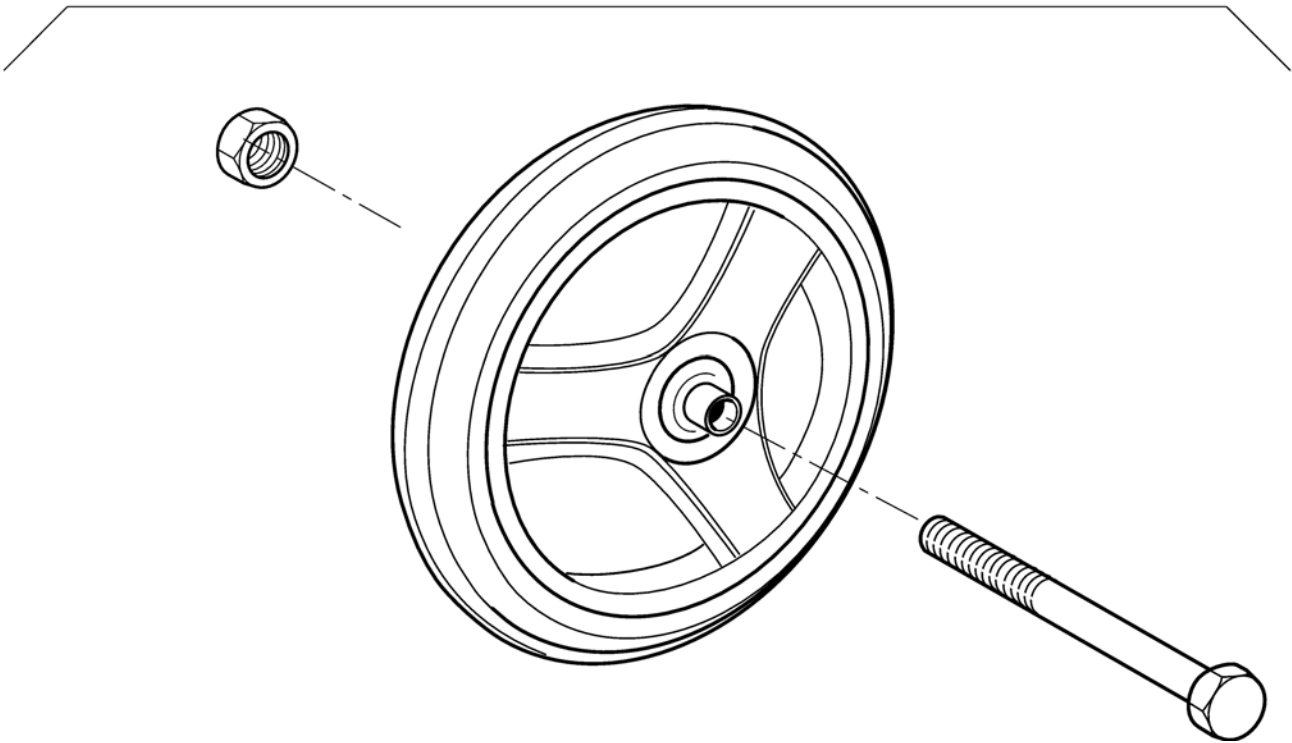


Pos.	Part number	Description	Valid from → until	Qty
1	1450173	Castor 120mm (5" x 1.1/4"), Solid, Cpl.		2

24" Standard Wheels

24"

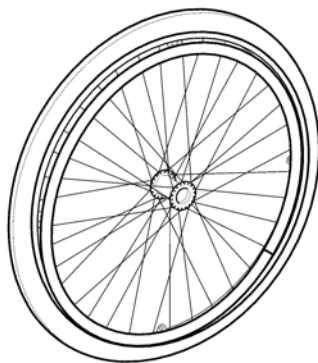

	1"	1"	1.3/8"	1.3/8"
	1	2	3	4
	5	6	7	8

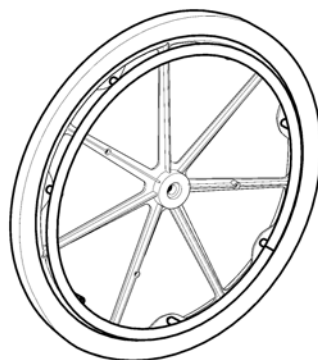



	1.3/8"	1.3/8"
	9	10
	11	12

19713349

Pos.	Part number	Description	Valid from → until	Qty
1	1498425	24" x 1" wheel, spoked, standard hub, anodized handrim, notched pneumatic tyre, without axle Ø12		2
2	1498431	24" x 1" spoked + anodized handrim + non marking pneumatic tyre > 03/2007		2
2	1498431-0618	24" x 1" spoked champagne + anodized handrim + non marking		2
3	1498423	24" x 1.3/8" spoked + anodized handrim + pneumatic tyre > 03/2007		2
4	1498263	24" x 1.3/8" wheel, spoked, standard hub, anodized handrim, solid tyre, without axle Ø12		2
5	1498426	24" x 1" spoked + anti-slip handrim + notched pneumatic tyre > 03/2007		2
6	1534211-0618	24" x 1" spoked champagne + anti-slip handrim + non marking		2
7	1498424	24" x 1.3/8" spoked + anti-slip handrim + pneumatic tyre > 03/2007		2
8	1498422	24" x 1.3/8" spoked + anti-slip handrim + solid tyre > 03/2007		2
9	1541973	24 Composite Wheel +Pneu Tyre + Alu H/rim	20100923 →	2
9	0572-05909	24" x 1.3/8" composite + anodized handrim + pneumatic tyre	→ 20100922	2
10	1541971	24 Composite Wheel +Solid Tyre + Alu H/rim	20100923 →	2
10	0572-05908	24" x 1.3/8" composite + anodized handrim + solid tyre	→ 20100922	2
11	1541974	24" Composite Wheel +Pneu Tyre + Anti Slip Ha	20100923 →	2
11	0572-05193	24" x 1.3/8" composite + anti-slip handrim + pneumatic tyre	→ 20100922	2
12	1541972	24" Composite Wheel +Solid Tyre + Anti Slip H	20100923 →	2
12	0572-05195	24" x 1.3/8" composite + anti-slip handrim + solid tyre	→ 20100922	2

24" Heavy Duty Wheels


24"

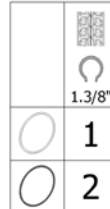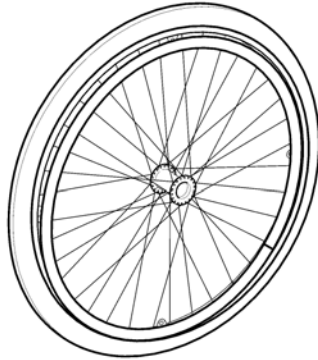

19713331

Pos.	Part number	Description	Valid from → until	Qty
1	1498486	24" x 1.3/8" pneumatic tyre, spoked rim, anodized handrim, Heavy Duty hub type Ø12		2
2	1498492	H D 24x 1 3/8 spoked + pneumatic + anti-slip hand rim		2


24" Standard Wheels accessories

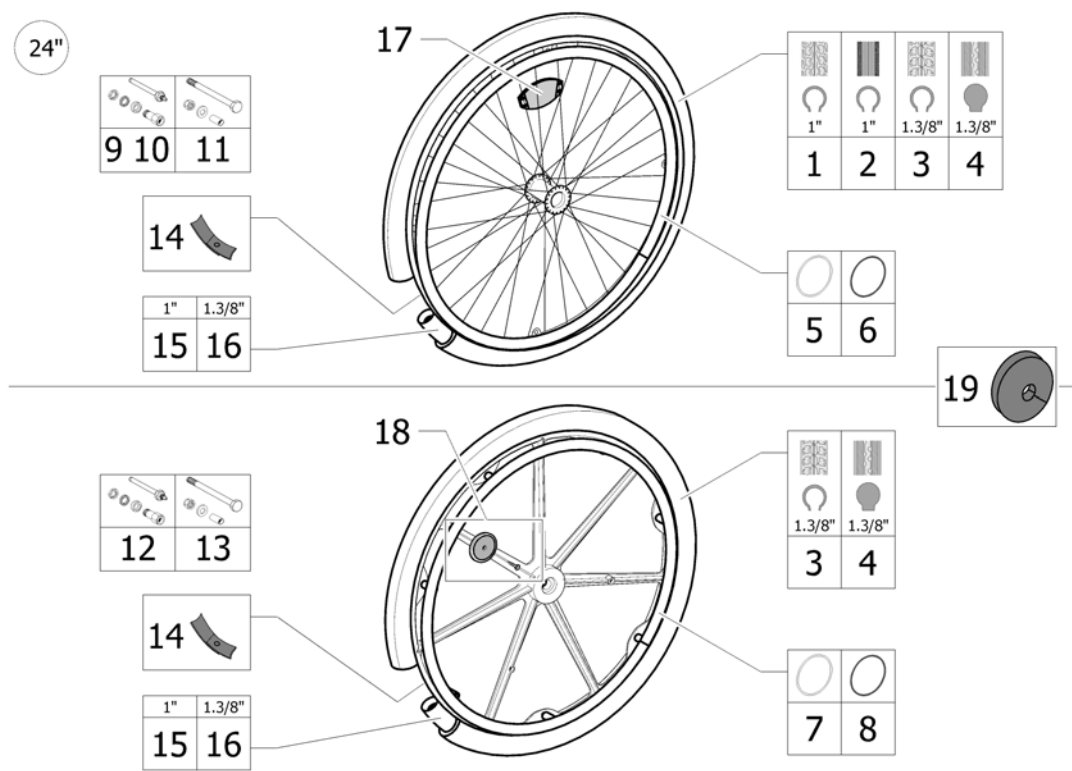

Pos.	Part number	Description	Valid from → until	Qty
1	0036-30013	24" x 1" notched pneumatic tyre		2
2	1026138	24" x 1" non marking pneumatic tyre		2
3	0036-30001	24" x 1.3/8" pneumatic tyre		2
4	0036-10013	24" x 1.3/8" solid tyre		2
5	1505291	24" anodized handrim 4 fixations		2
6	1505298	24" anti-slip handrim 4 fixations		2
7	1545554	24" anodized handrim 7 fixations v2010 <i>Replace 1431317</i>	20100923 →	2
7	1431317	24" anodized handrim 7 fixations <i>Replaced by 1545554</i>	→ 20100922	2
8	1545555	24" anti-slip handrim 7 fixations <i>Replace 1431318</i>	20100923 →	2
8	1431318	24" anti-slip handrim 7 fixations <i>Replaced by 1545555</i>	→ 20100922	2
9	1505288	QR axle with axle support Ø12		2
10	1505290	QR axle for hub brake spoked wheels		2
11	1505289	Fixed axle for spoked wheels > 03/2007		2
12	1434129	QR axle for composite wheels		2
13	1431279	Fixed axle for composite wheels		2
14	0036-40003	24" rim tape		2
15	1032240	24" x 1" inner tube		2
16	0036-20001	24" x 1.3/8" inner tube		2
17	N90003	Kit reflectors orange for spoked wheels		2
18	9037-30001	Kit reflectors orange for composite wheels		2
19	0789-04400	Spoke guards 24"		1

22" Standard Wheels

22"

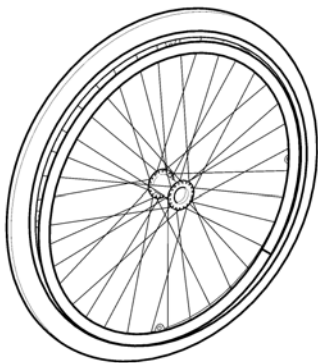



		
		
	1.3/8"	1.3/8"
	1	2
	3	4

19713331

Pos.	Part number	Description	Valid from → until	Qty
1	1498434	22" Pneumatic Basic/19" Alu. Handrim		2
2	1498432	22" x 1.3/8" spoked + anodized handrim + solid tyre > 03/2007		2
3	1498435	22" x 1.3/8" spoked + anti-slip handrim + pneumatic tyre > 03/2007		2
4	1498433	22" x 1.3/8" spoked + anti-slip handrim + solid tyre > 03/2007		2

22" Heavy Duty Wheels

22"

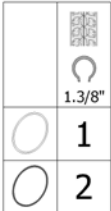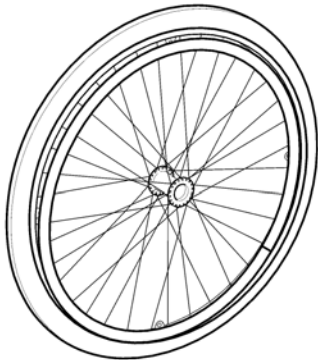


19713331

Pos.	Part number	Description	Valid from → until	Qty
1	1498493	H D 22x 1 3/8 spoked + pneumatic + anod. hand rim		2
2	1498494	H D 22x 1 3/8 spoked + pneumatic + anti-slip hand rim		2

22" Standard Wheels accessories

22"

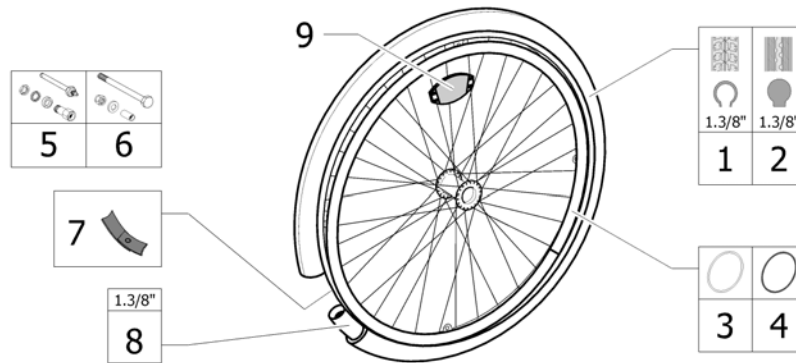


199133362

Pos.	Part number	Description	Valid from → until	Qty
1	0036-30035	22" x 1.3/8" pneumatic tyre		2
2	0036-10014	22" x 1.3/8" solid tyre grey		2
3	1505299	22" anodized handrim 4 fixations		2
4	1505920	22" anti-slip handrim 4 fixations		2
5	1505288	QR axle with axle support Ø12		2
6	1505289	Fixed axle for spoked wheels > 03/2007		2
7	0036-40002	22" rim tape		2
8	0036-20007	22" x 1.3/8" inner tube		2
9	N90003	Kit reflectors orange for spoked wheels		2

Invacare® Action 3 NG | Rear Wheels

20" Standard Wheels Compact

20"

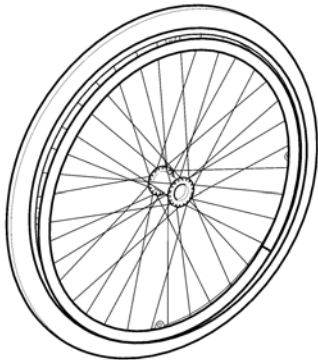


		
		
	1.3/8"	1.3/8"
	1	2

18951324164

Pos.	Part number	Description	Valid from → until	Qty
1	1445041	20" Pneumatic Basic/19" Alu. Handrim		2
2	1445042	20" Solid Basic/19" Alu. Handrim		2



20" Standard Wheels accessories Compact

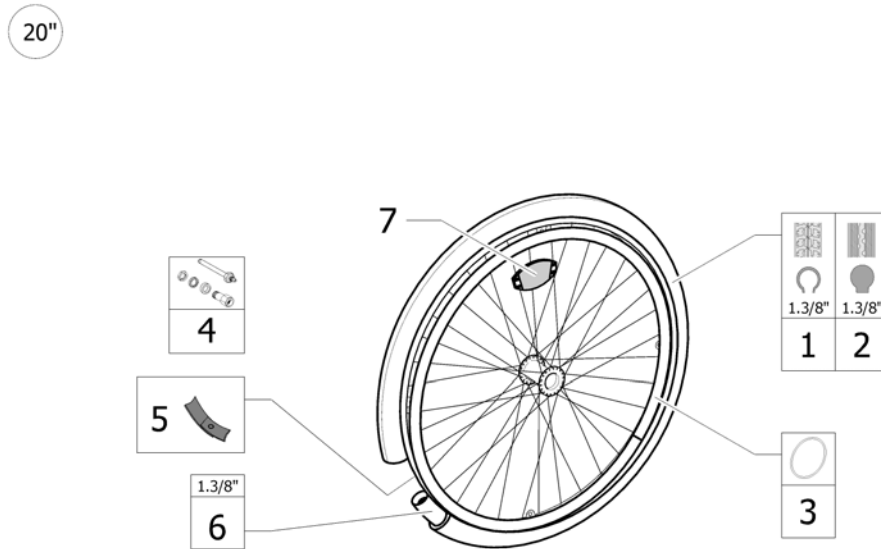

18951324185

Pos.	Part number	Description	Valid from → until	Qty
1	M11009	Tyre - 20" x 1 3/8"		2
2	1TYR2002	Solid Tyre - 20" x 1 3/8"		2
3	1507241	20" Anodised Handrim		2
4	1505288	QR axle with axle support Ø12		2
5	1507198	20" Rim Tape		2
6	0036-20006	20" Inner Tube		2
7	N90003	Kit reflectors orange for spoked wheels		2

24" / 22" Hub brake wheels

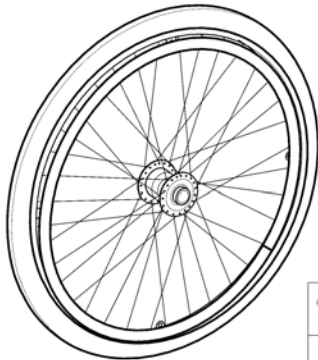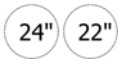


		
	1.3/8"	1.3/8"
24"	1	2
22"	3	4

ISP1535353

Pos.	Part number	Description	Valid from → until	Qty
1	1498428	24" x 1.3/8" hub brake spoked + anodized handrim + pneumatic tyre >06/2007		2
2	1498427	24" x 1.3/8" hub brake spoked + anodized handrim + solid tyre > 06/2007		2
3	1498466	22" x 1.3/8" hub brake spoked + anodized handrim + pneumatic tyre >06/2007		2
4	1498461	22" x 1.3/8" hub brake spoked + anodized handrim + solid tyre > 06/2007		2
5	1505290	QR axle for hub brake spoked wheels		2


Rear Wheel Supports

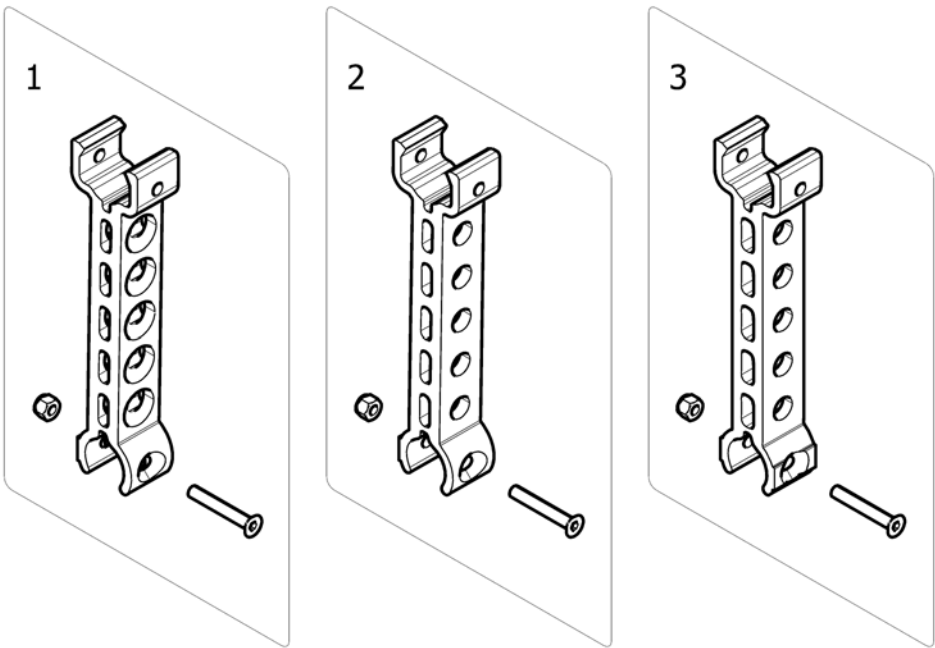

15353334

Pos.	Part number	Description	Valid from → until	Qty
1	1534886	Rear wheel support for QR axle		2
2	1534887	Rear wheel support for fixed axle		2
3	1535334	Rear wheel support for transit		2


12" Rear Wheels

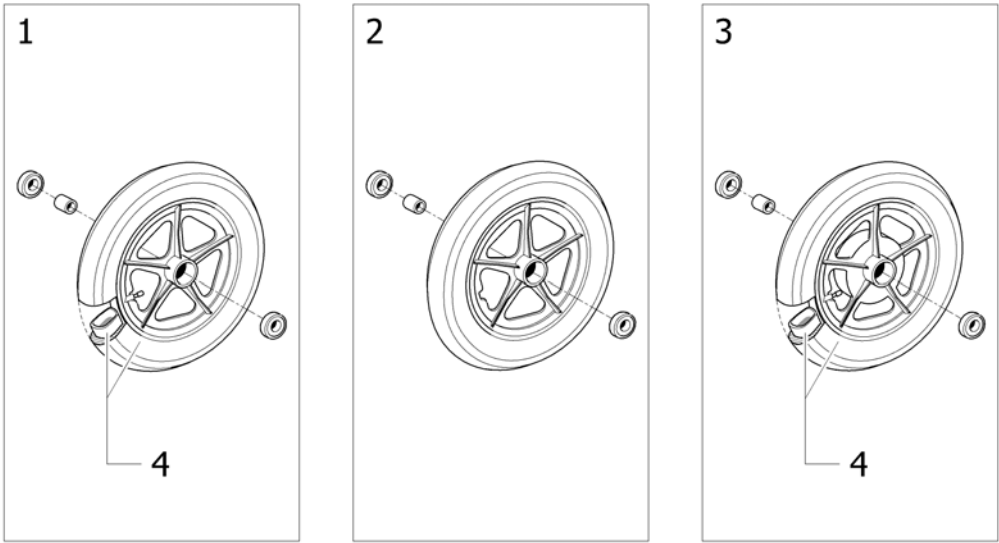

1895133305

Pos.	Part number	Description	Valid from → until	Qty
1	6001789	12" PNEUMATIC COMPLETE WHEEL		2
2	1535037	12" rear wheel, composite, standard hub, solid tyre,without axle Ø12 <i>Replace 6001790</i>	20091015 →	2
2	1530448	Pair of 12.1/2" solid wheels	20041125 → 20091014	1
3	0572-03002	Complete 70 Hub Brake Composite 12.1/2" Wheel		2
4	1431313	12" transit Pneumatic Tyre + Inner tube		2


Rear wheel support transit

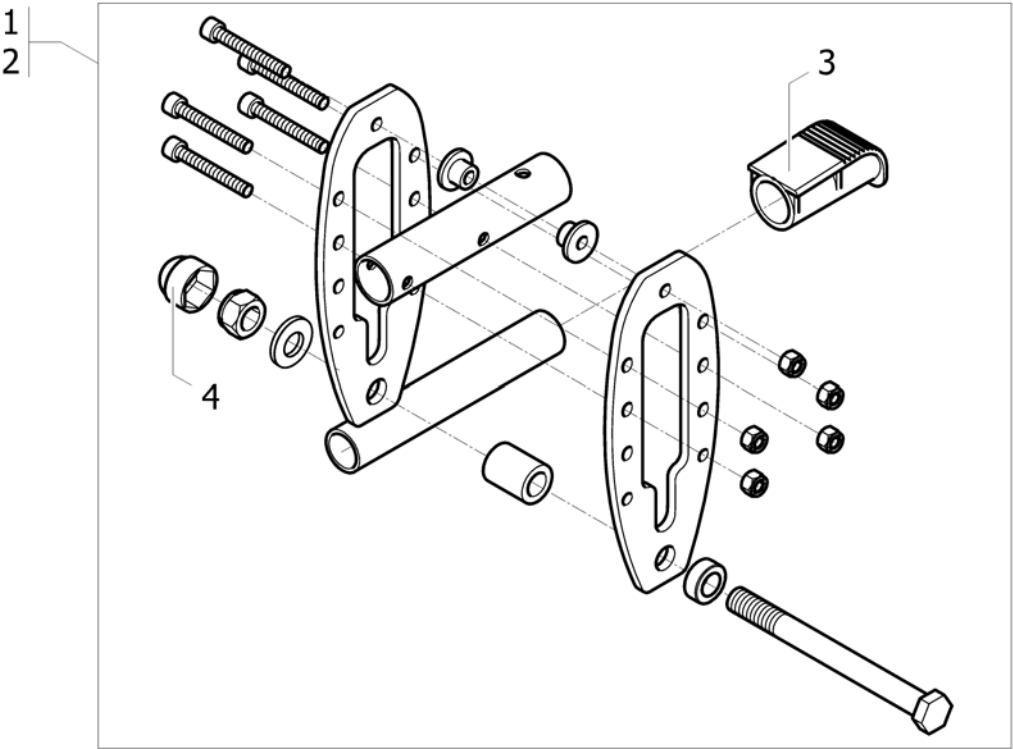

199133357


Pos.	Part number	Description	Valid from → until	Qty
1	1431280	Standard Transit Rear Axle		2
2	1431281	Hub Brake Transit Rear Axle		2
3	0021-20011	Black end cap for tipping aid		2
4	0021N20030	C54 Plastic Cap BlackRear Whl		2

Pump

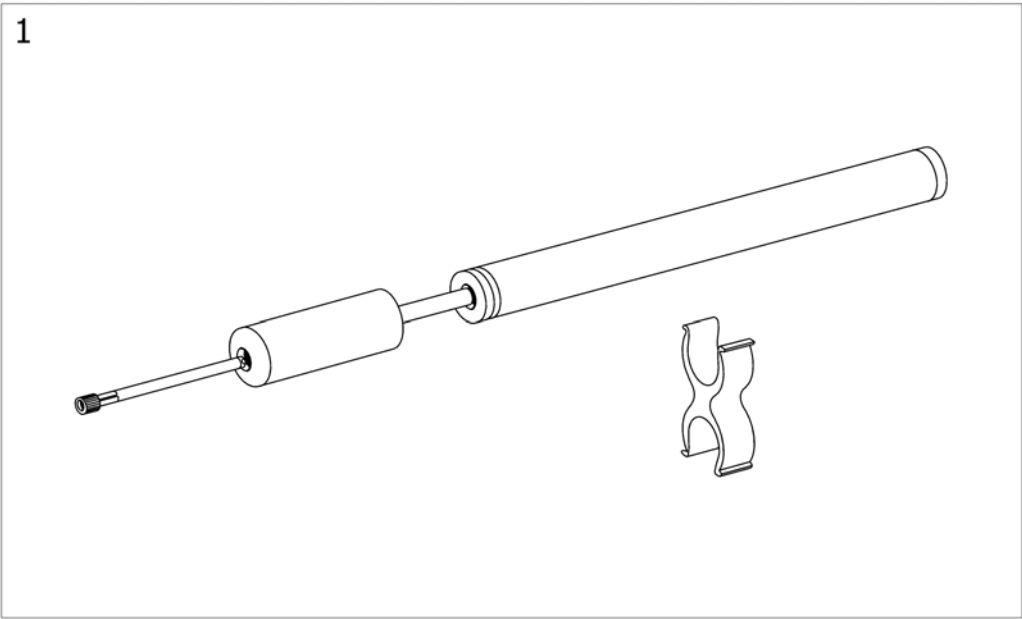

Pos.	Part number	Description	Valid from → until	Qty
1	1419910	Kit pump + support <i>Replace 1484715</i>		1


Push Brakes : Standard And With Extension

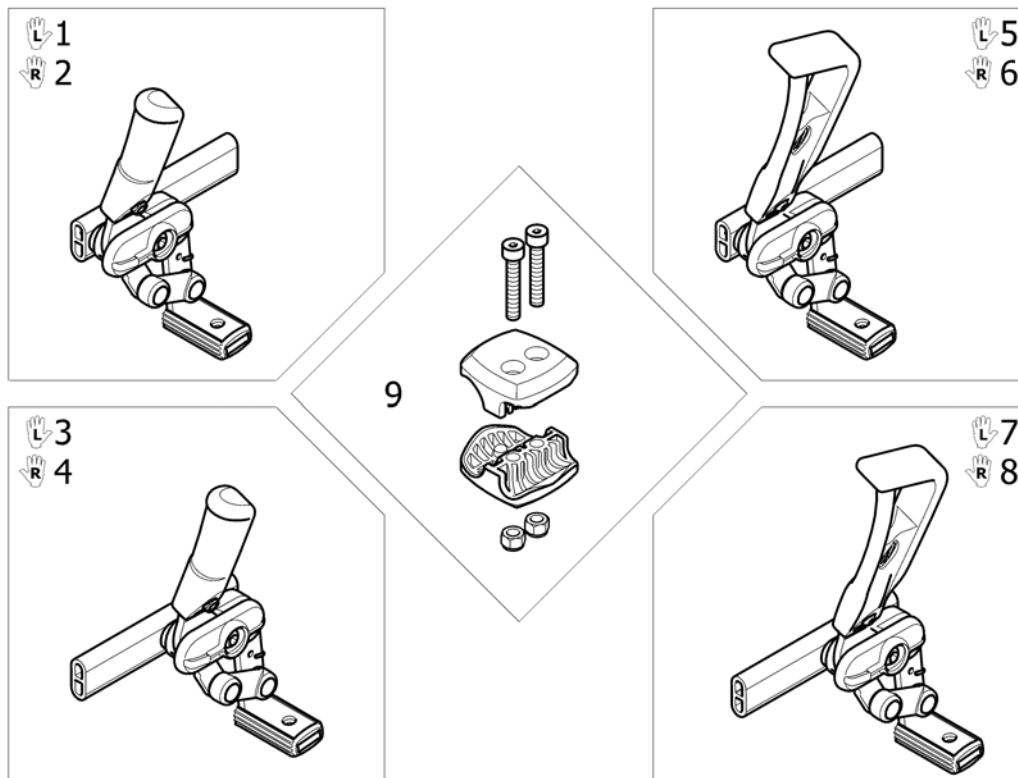

12951405728


Pos.	Part number	Description	Valid from → until	Qty
1	1425634	Brake Lh Long		1
2	1425635	Brake Rh Long		1
3	1425642	Self Propel Brake Assembly LH		1
4	1425643	Self Propel Brake Assembly RH		1
5	1425602	Extended Brake Lh Long		1
6	1425603	Extended Brake Rh Long		1
7	1425640	Self Propel Extended Brake Assembly LH		1
8	1425641	Self Propel Extended Brake Assembly RH		1
9	1423107	Kit brake composite clamp		2

Push brake Transit

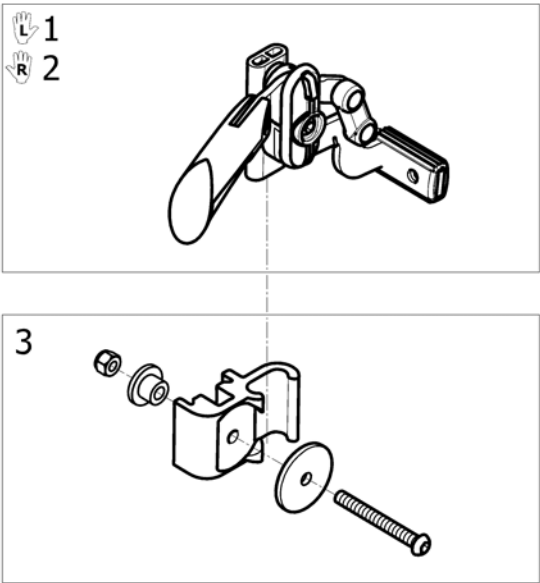


ISPS1535356

Pos.	Part number	Description	Valid from → until	Qty
1	1431992	Rh Transit Push to Lock Brake		1
2	1431991	Lh Transit Push to Lock Brake		1
3	1430853	Transit Brake Support Kit		2

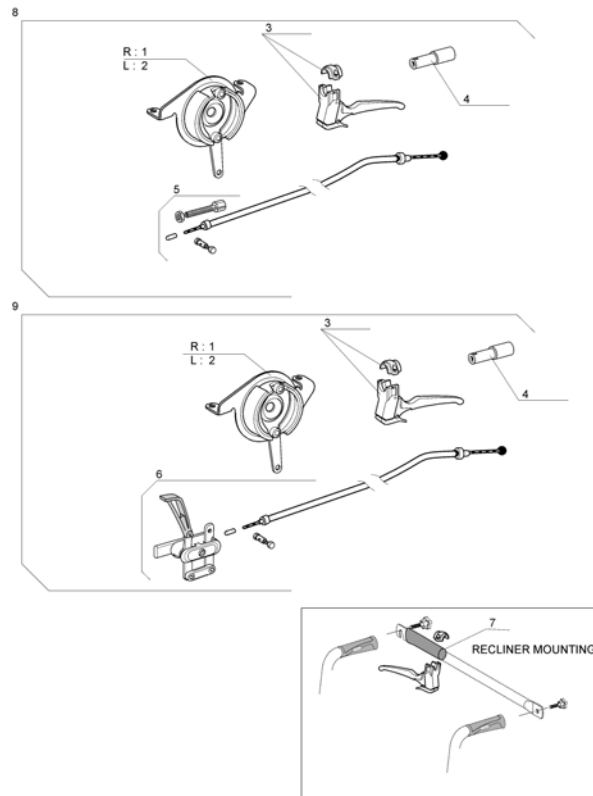


Pos.	Part number	Description	Valid from → until	Qty
1	1423292	Rh Hub Brake Plate 70mm		1
2	1423293	Lh Hub Brake Plate 70mm		1
3	1420527	Black Brake Lever dia 22mm		2
4	1430412	Back brace insert for fixed backrest tube		2
5	1430204	Attendant Hub Brakes Complete Mechanism (pair)		1
6	1430205	Dual hub brakes complete mechanism (pair)	→ 20061231	1
7	1434626	Backbrace Handle		2
8	1445674	Attendant Hub Brakes Complet Kit (Without Wheel & Axle)		1
9	1445675	Dual Hub Brakes Complet Kit (Without Wheel & Axle)	→ 20061231	1
-	AP1543359	Transformation kit hub brake attendant Action 3 NG 12 pneumatic (including wheels)		1
-	AP1543360	Transformation kit hub brake attendant Action 3 NG 24 pneumatic (including wheels)		1
-	AP1543361	Transformation kit hub brake attendant Action 3 NG 24 solid (including wheels)		1
-	AP1543362	Transformation kit hub brake attendant Action 3 NG 22 pneumatic (including wheels)		1
-	AP1543363	Transformation kit hub brake attendant Action 3 NG 22 solid (including wheels)		1

Oad Wire Control Brake

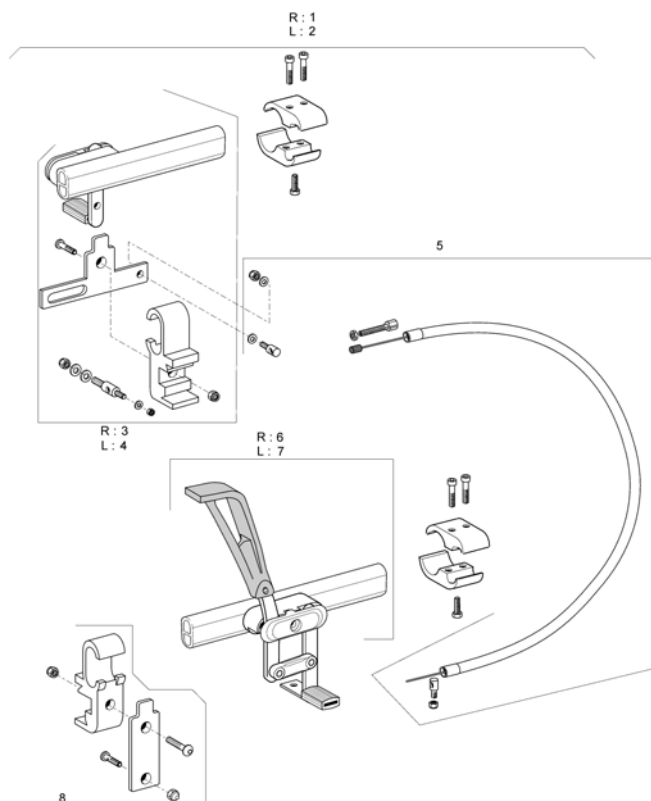

Pos.	Part number	Description	Valid from → until	Qty
1	1430202	One Arm Brake Cpl, Ri		1
2	1430203	One Arm Brake Cpl, Le		1
3	1433462	Brake Unit Cpl, Ri, One Arm Brake		1
4	1433461	Brake Unit Cpl, Le, One Arm Brake		1
5	1433464	Wire Control Cpl, One Arm Brake		1
6	1430254	Brake Ri, One Arm Brake		1
7	1430253	Brake Le, One Arm Brake		1
8	1433463	Clamp Kit, Wire Control		1


Lapt belts

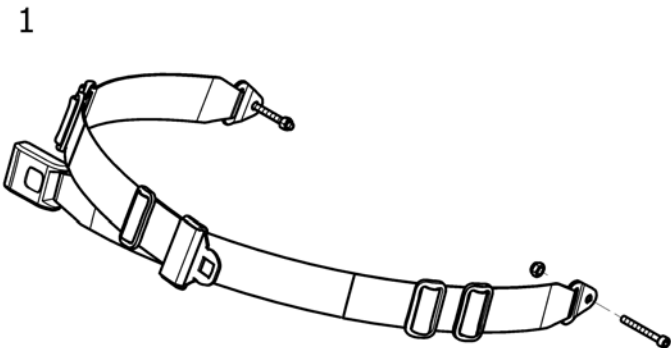

0572-05389

Pos.	Part number	Description	Valid from → until	Qty
1	0572-05389	Pelvic Belt, Spirea4Ng <i>Replace 0572-02049</i>		1


Rear Anti Tipper And Tipping Tube

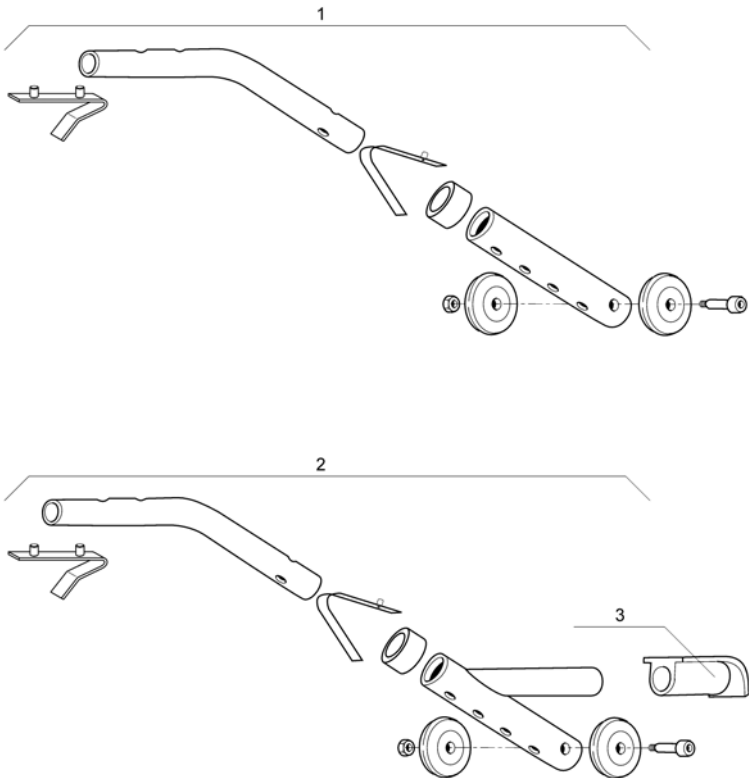

Pos.	Part number	Description	Valid from → until	Qty
1	0572-05848	Kit anti-tippers		2
2	0572-05849	Tipping & Rear Anti Tipper Tubes		2
3	0021-20011	Black end cap for tipping aid		2





Anti-tip

1

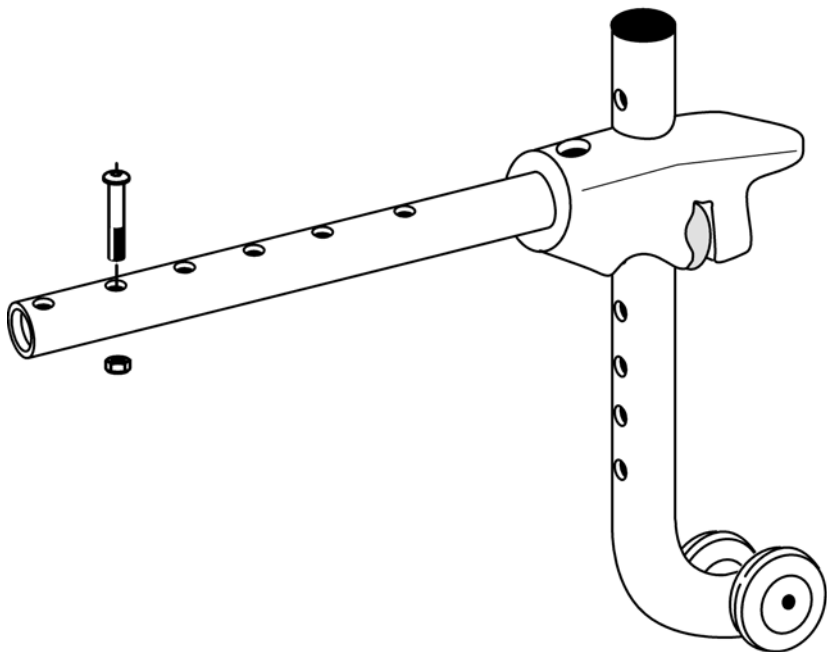

Pos.	Part number	Description	Valid from → until	Qty
1	1484890	Anti-tippers A3/A4 types		2


Reflectors

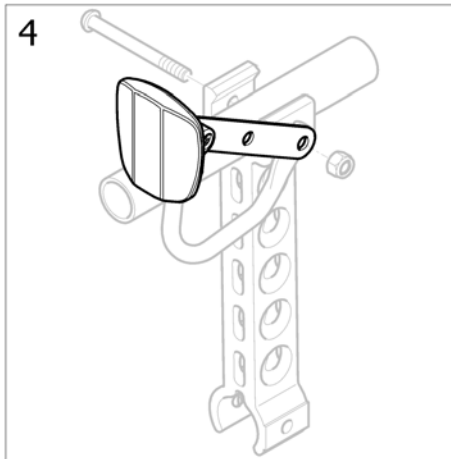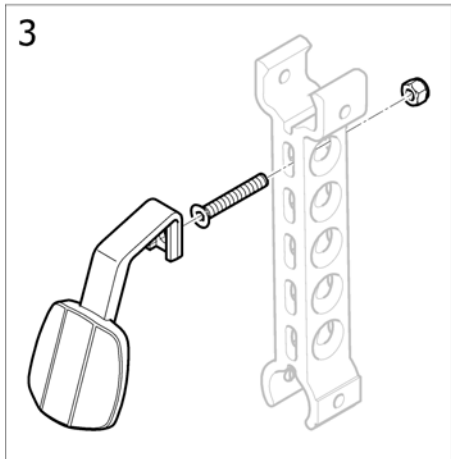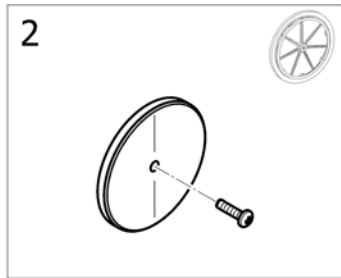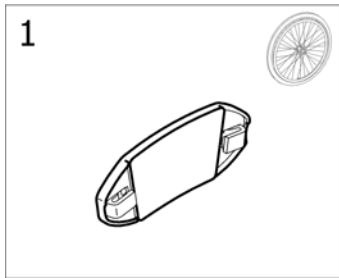

ISPS1495928

Pos.	Part number	Description	Valid from → until	Qty
1	N90003	Kit reflectors orange for spoked wheels		2
2	9037-30001	Kit reflectors orange for composite wheels		2
3	1430184	Frame Reflectors Kit		2
4	1450569	A4 Frame Reflectors Kit		2


Transfer Kit

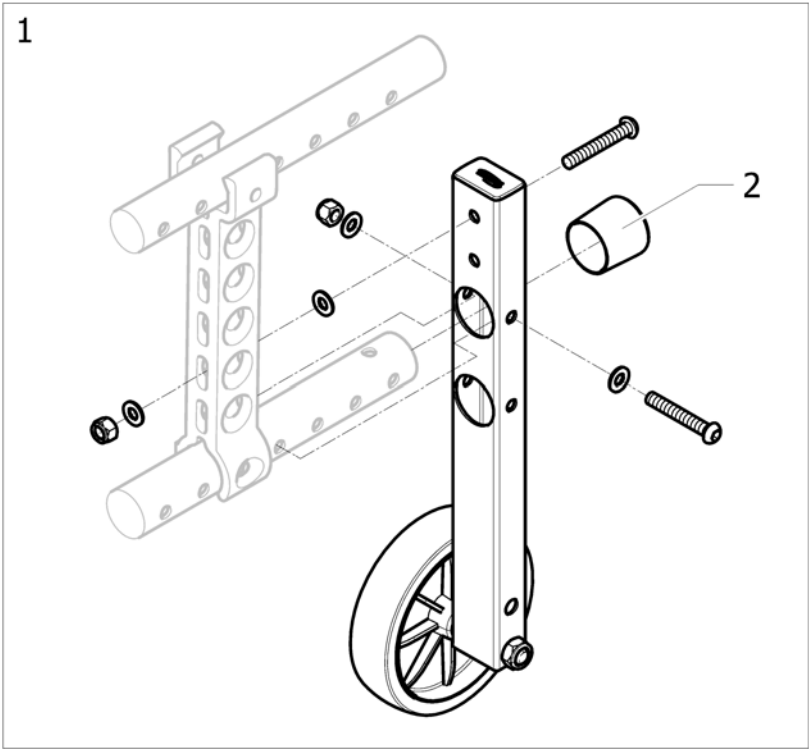


195149000

Pos.	Part number	Description	Valid from → until	Qty
1	1492139	Pair of anti-barrier wheels kit		1
2	0021N05386	Frame cap black 1		2

Standard Headrest

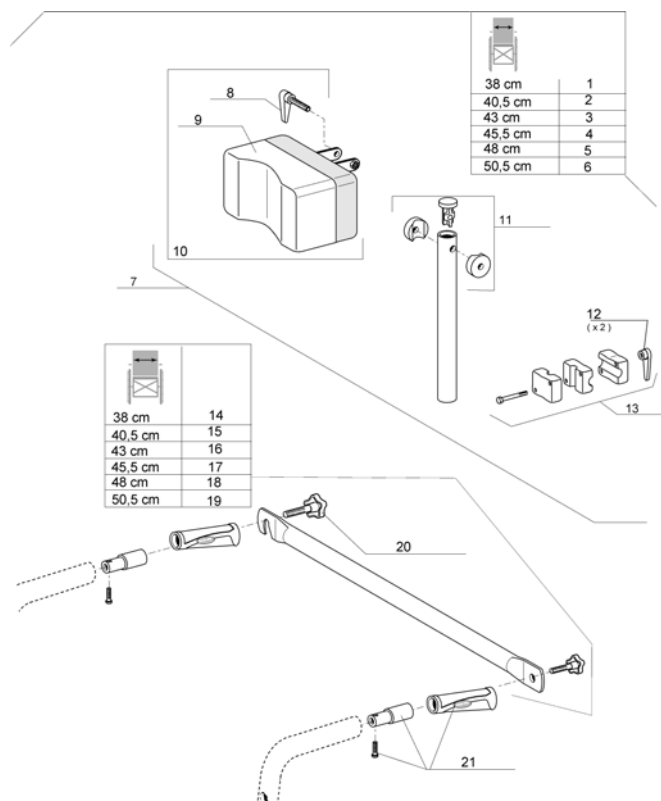


Pos.	Part number	Description	Valid from → until	Qty
1	1433028	Headrest + Backbrace + Fixation Set W 38Cm		1
2	1433029	Headrest + Backbrace + Fixation Set W 40,5Cm		1
3	1433030	Headrest + Backbrace + Fixation Set W 43Cm		1
4	1433031	Headrest + Backbrace + Fixation Set W 45,5Cm		1
5	1433032	Headrest + Backbrace + Fixation Set W 48Cm		1
6	1433033	Headrest + Backbrace + Fixation Set W 50,5Cm		1
7	1433702	Comfort Headrest + Support Tube		1
8	1428595	Indexable Lever; M6X40		1
9	1420507	Headrest Pad Comfort Vinyl		1
10	1432623	Headrest Cushion + Fixation		1
11	1432622	Headrest Tube Kit		1
12	1428547	Indexable Lever; Female M6		1
13	1432617	Headrest Support Kit		1
14	1434099	Back brace with inserts and handles SW38		1
15	1434100	Back brace with inserts and handles SW40.5		1
16	1433707	Back brace with inserts and handles SW43		1
17	1433708	Back brace with inserts and handles SW45.5		1
18	1433716	Back brace with inserts and handles SW48		1
19	1434105	Back brace with inserts and handles SW50,5		1
20	0021-60029	Button Ø32 mm M6x16		2
21	1430412	Back brace insert for fixed backrest tube		2

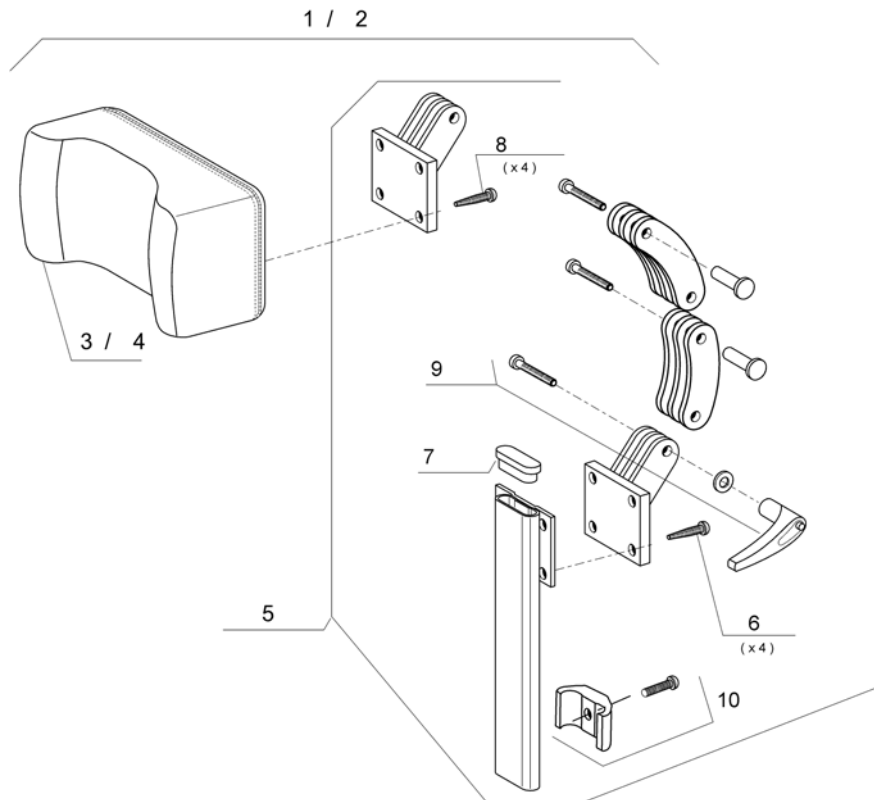Comfort Headrest

Pos.	Part number	Description	Valid from → until	Qty
1	1440900	Headrest Comfort (black)		1
2	1514954	Headrest comfort vinyl		1
3	1440903	Headrest Pad Comfort (black)		1
4	1420507	Headrest Pad Comfort Vinyl		1
5	1415942	Articulation headrest kit		1
6	1421921	Zinc st M5 x 8 TBCL screw		4
7	5716497	PLASTIC CAP		1
8	0029-40032	6.8 Zinc CHC M5 x 20 Screw		4
9	1440954	Articulation headrest fixing kit		1
10	1448661	Stop Cpl., Neck- & Headrest		1

Cushions


	SD40		SD45	
	SD40	SD45	SD40	SD45
SW38	1	4	7	10
SW40.5	1	4	7	10
SW43	2	5	8	11
SW45.5	2	5	8	11
SW48	3	6	9	12
SW50.5	3	6	9	12

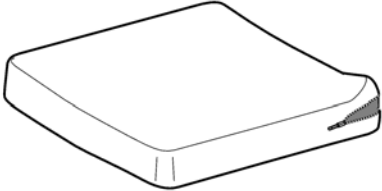

ISPS1535399


Pos.	Part number	Description	Valid from → until	Qty
1	0572-05013	Cushion Foam Trevira W 38 x D 40 cm		1
2	0572-05017	Cushion Foam Trevira W 43 x D 40 cm		1
3	1418615	Cushion Foam Trevira W 48 x D 40 cm		1
4	0572-05014	Cushion Foam Trevira W 38 x D 45 cm		1
5	0572-05018	Cushion Foam Trevira W 43 x D 45 cm		1
6	1418616	Cushion Foam Trevira W 48 x D 45 cm		1
7	1541977	Seat cushion foam nylon CW40 CD38		1
8	1541979	Seat cushion foam nylon CW40 CD43		1
9	1541981	Seat cushion foam nylon CW40 CD48		1
10	1541978	Cushion Foam Nylon W 38 x D 45 cm		1
11	1541980	Cushion Foam Nylon W 43 x D 45 cm		1
12	1541982	Cushion Foam Nylon W 48 x D 45 cm		1

Cushions Visco

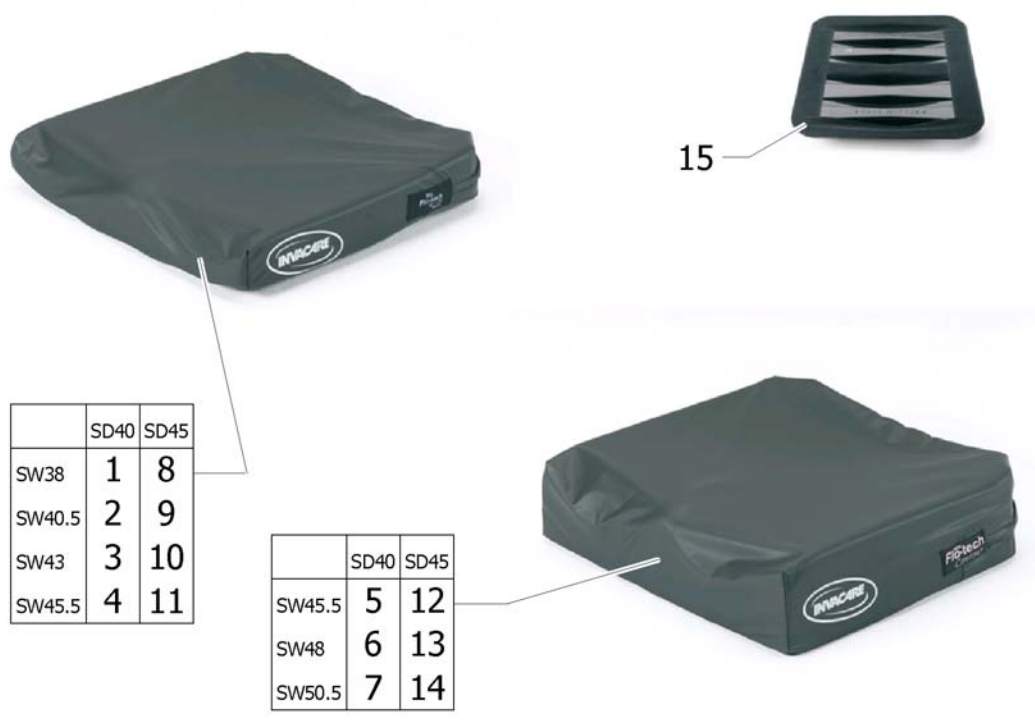


Pos.	Part number	Description	Valid from → until	Qty
1	1501111	Flo-tech Lite Visco - W38 x D40,5 x H7,8cm low memory firm MUW:100Kg (FLSB15/16/LMF/SGL)		1
2	1501109	Flo-tech Lite Visco - W40,5 x D40,5 x H7,8cm low memory firm MUW:100Kg (FLSB16/16/LMF/SGL)		1
3	1501107	Flo-tech Lite Visco - W43 x D40,5 x H7,8cm low memory firm MUW:100Kg (FLSB17/16/LMF/SGL)		1
4	1523826	Flo-tech Lite Visco - W45,5 x D40,5 x H7,8cm low memory firm MUW:100Kg (FLSB18/16/LMF/SGL)		1
5	1501114	Flo-tech Contour Visco - W45,5 x D40,5 x H12cm low memory firm MUW:100Kg (FCSB18/16/LMF/SGL)		1
6	1501116	Flo-tech Contour Visco - W48 x D40,5 x H12cm low memory firm MUW:100Kg (FCSB19/16/LMF/SGL)		1
7	1501118	Flo-tech Contour Visco - W50,5 x D40,5 x H12cm low memory very firm MUW:125Kg (FCSB20/16/LMV/SGL)		1
8	1525532	Flo-tech Lite Visco - W38 x D45,5 x H7,8cm low memory firm MUW:100Kg (FLSB15/18/LMF/SGL) <i>Replace 1501112</i>		1
9	1525533	Flo-tech Lite Visco - W40,5 x D45,5 x H7,8cm low memory firm MUW:100Kg (FLSB16/18/LMF/SGL) <i>Replace 1501110</i>		1
10	1525534	Flo-tech Lite Visco - W43 x D45,5 x H7,8cm low memory firm MUW:100Kg (FLSB17/18/LMF/SGL) <i>Replace 1501108</i>		1
11	1525535	Flo-tech Lite Visco - W45,5 x D45,5 x H7,8cm low memory firm MUW:100Kg (FLSB18/18/LMF/SGL)		1
12	1526608	Flo-tech Contour Visco - W45,5 x D45,5 x H12cm low memory firm MUW:100Kg (FCSB18/18/LMF/SGL) <i>Replace 1501115</i>		1
13	1525020	Flo-tech Contour Visco - W48 x D45,5 x H12cm low memory very firm MUW:125Kg (FCSB19/18/LMV/SGL) <i>Replace 1501117</i>		1
14	1525021	Flo-tech Contour Visco - W50,5 x D45,5 x H12cm low memory very firm MUW:125Kg (FCSB20/18/LMV/SGL) <i>Replace 1501119</i>		1
15	AP1487165	Seat cushion rigidiser		1

Trunk Support Comfort Dartex

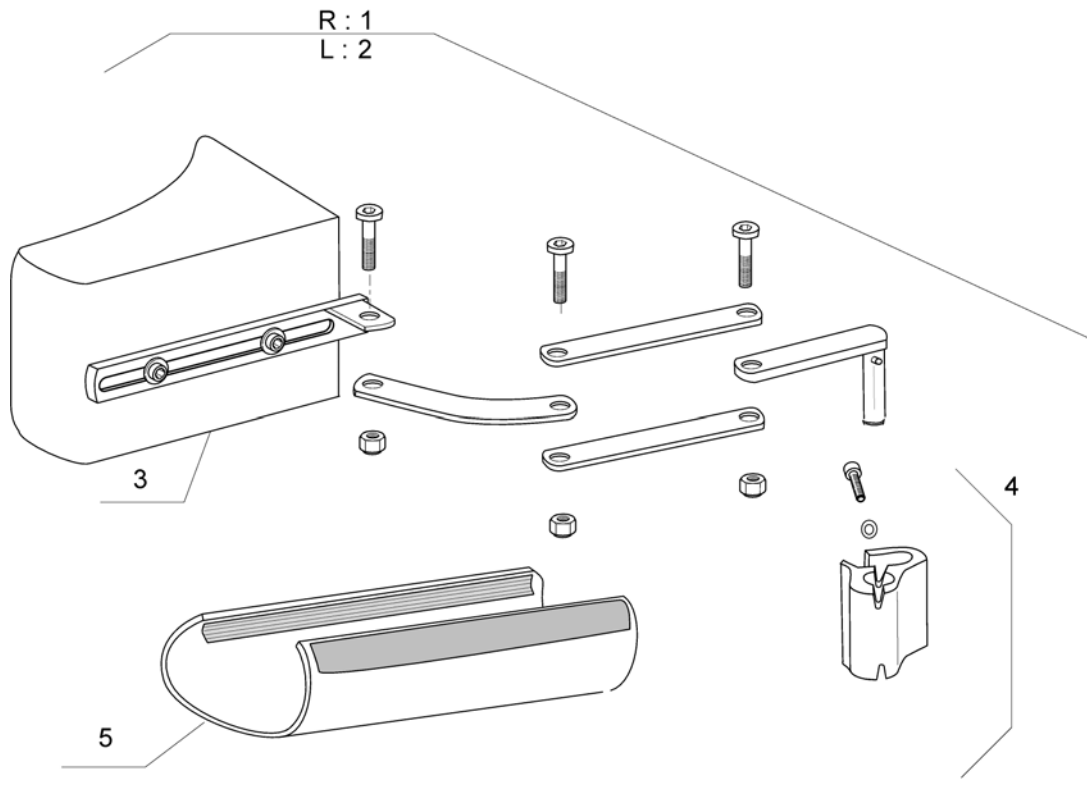


Pos.	Part number	Description	Valid from → until	Qty
1	1436044	Complet Rh Trunk Support Comfort Dartex	→ 20081231	1
2	1436032	Complet Lh Trunk Support Comfort Dartex	→ 20081231	1
3	1440896	Trunk Support Pad Dartex	→ 20081231	2
4	1442517	Trunk Support Fixation	→ 20081231	2
5	1437642	Cover, Trunksupport Swingaway	→ 20081231	2

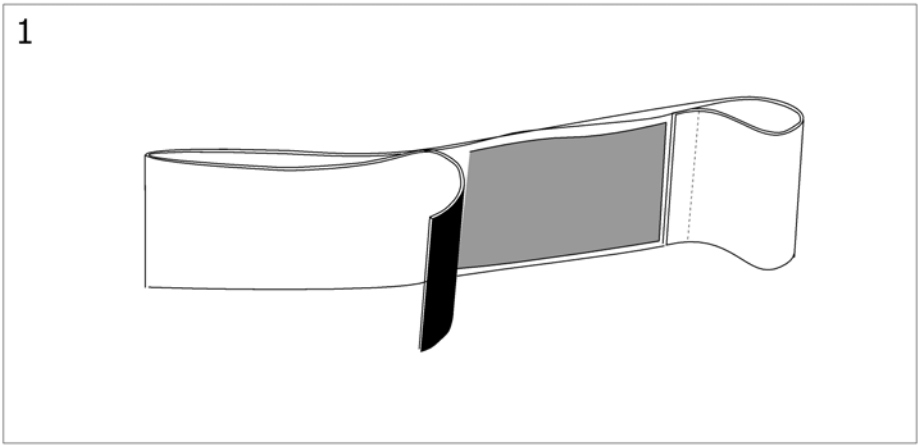



12951-493803

Pos.	Part number	Description	Valid from → until	Qty
1	1433580	Adjustable Calf Strap w 38cm 48 cm		1

Heels strap

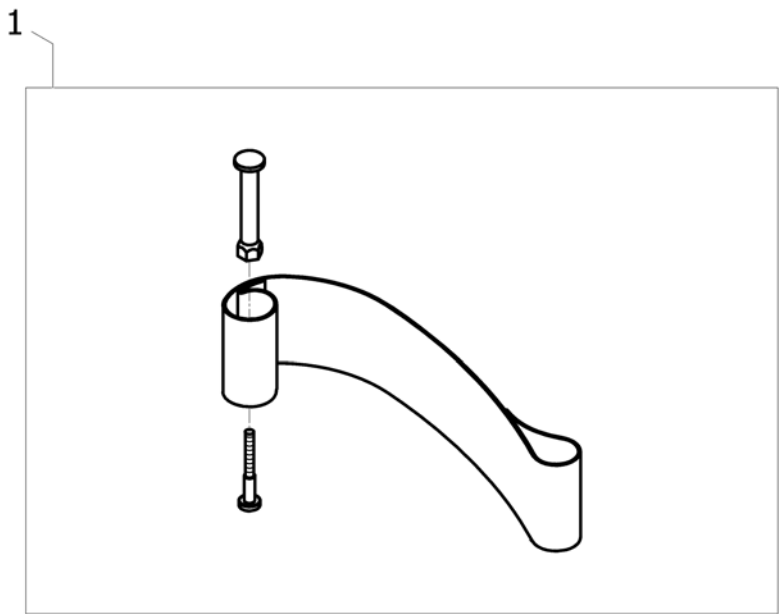


Pos.	Part number	Description	Valid from → until	Qty
1	1492315	Heels Strap Adjustable <i>Replace 1534891</i>		2

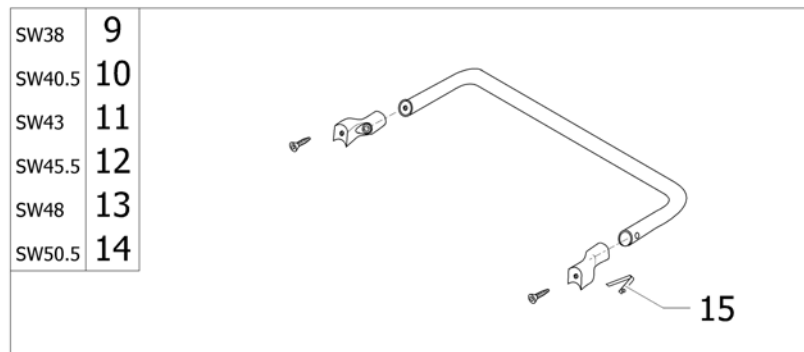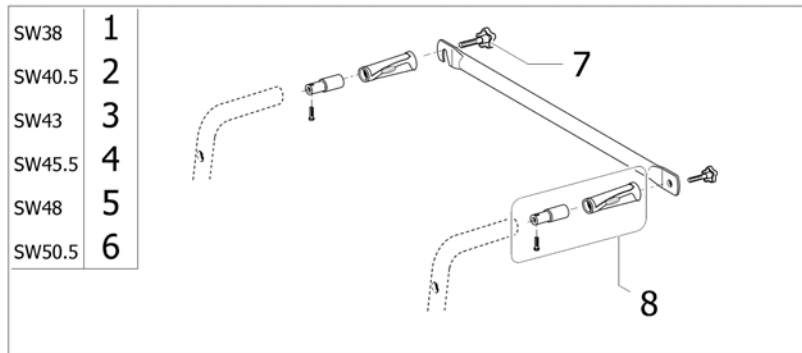Back Brace

ISPS1543302


Pos.	Part number	Description	Valid from → until	Qty
1	1434099	Back brace with inserts and handles SW38		1
2	1434100	Back brace with inserts and handles SW40.5		1
3	1433707	Back brace with inserts and handles SW43		1
4	1433708	Back brace with inserts and handles SW45.5		1
5	1433716	Back brace with inserts and handles SW48		1
6	1434105	Back brace with inserts and handles SW50,5		1
7	0021-60029	Button Ø32 mm M6x16		2
8	1430412	Back brace insert for fixed backrest tube		2
9	1450575	Fixed Backrest Back Brace Kit Assy W 38Cm		1
10	1450576	Fixed Backrest Back Brace Kit Assy W 40,5Cm		1
11	1450577	Fixed Backrest Back Brace Kit Assy W 43Cm		1
12	1450578	Fixed Backrest Back Brace Kit Assy W 45,5Cm		1
13	1450562	Fixed Backrest Back Brace Kit Assy W 48Cm		1
14	1450451	Fixed Backrest Back Brace Kit Assy W 50,5Cm		1
15	0027-10003	Stud spring Ø6mm		1

One arm drive

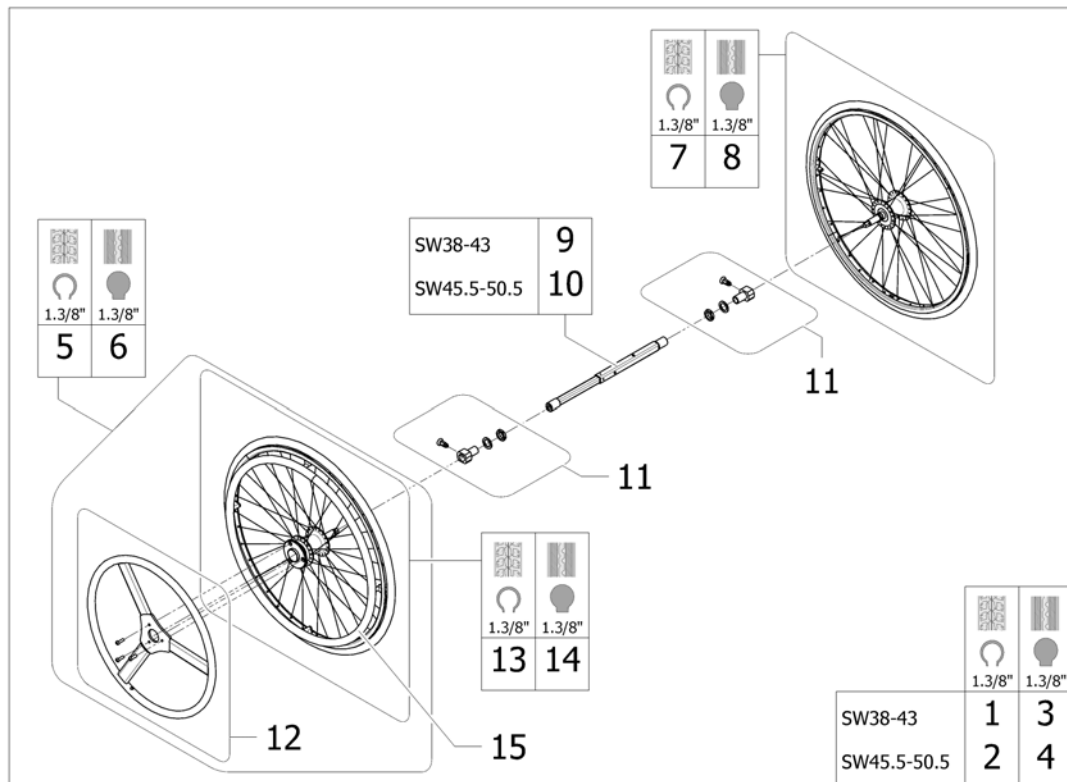

Pos.	Part number	Description	Valid from → until	Qty
1	1513269	Dual handrim DHR pneumatic kit assy 38/43 cm		1
2	1513270	Dual handrim DHR pneumatic kit assy 45,5/50,5 cm		1
3	1513271	Dual handrim DHR solid kit assy 38,5/43 cm		1
4	1513272	Dual handrim DHR solid kit assy 45,5/50,5 cm		1
5	1509382	24"Pneu, With Rim, One Hand Drive Wheel		1
6	1509436	24" DHR active wheel solid		1
7	1508491	24"Pneu, One Hand Drive Wheel		1
8	1508490	24" DHR passive wheel solid		1
9	1511769	Telescopic Shaft 270-385		1
10	1508487	Telescopic Shaft 345-460		1
11	1513658	DHR wheel axle support complet		2
12	1508484	Small handrim + fixations		1
13	1508485	24" DHR active wheel pneumatic w/o small handrim		1
14	1508486	24" DHR active wheel solid w/o small handrim		1
15	1514223	24" Aluminium handrim + 6 fixations		1


Hand Control For Oad By Lever

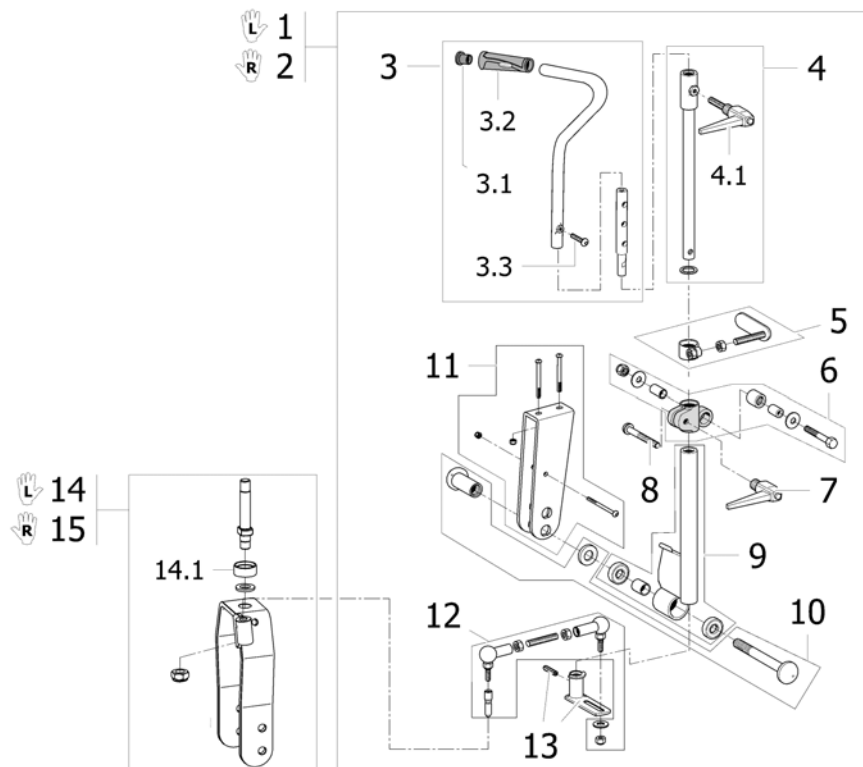

15353367


Pos.	Part number	Description	Valid from → until	Qty
1	1432936	Oad By Lever Hand Control Lh Completed Kit		1
2	1432935	Oad By Lever Hand Control Rh Completed Kit		1
3	1432624	Oad by lever hand control assy		1
3.1	1418112	Black cap to be inserted Ø22x15		1
3.2	1425473	Invacare Push Handle <i>Replace 1435421</i>		1
3.3	0029-40049	Cbhc screw m8x30; 8.8		1
4	1432625	Oad by lever hand lever shaft		1
4.1	0021-70002	Indexable lever 63mm M8x16		1
5	1432627	Oad by lever brake shoe assy		1
6	1432998	Oad by lever adjustable sleeve assy		1
7	0021-70011	Indexable lever 63 m8		1
8	0029-40252	Clamp screw m8 x 1 1/4"		1
9	1432999	Oad By Lever Equalizer		1
10	1433000	Equalizer Completed Axle		1
11	1433003	Equalizer Support + Fixation		1
12	1534241	Oad By Lever Ball Joint Kit		1
13	1433002	Oad by lever steering bush + fixation		1
14	1535369	Oad By Lever Fork And Axle Kit Lh		1
14.1	1529516-0618	Fork Ring "Champagne"		1
15	1535368	Oad By Lever Fork And Axle Kit Rh		1

Oad By Lever

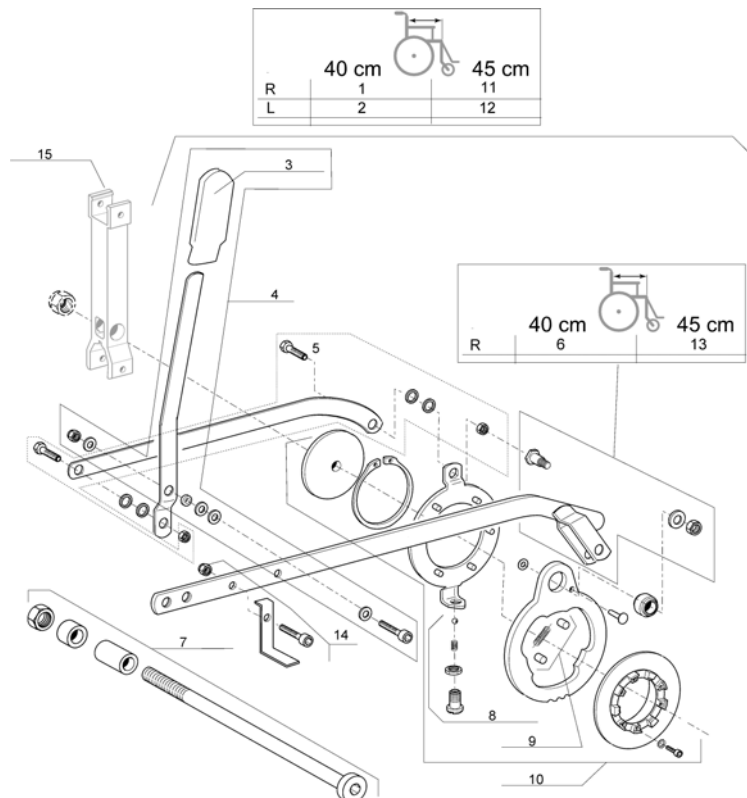


Pos.	Part number	Description	Valid from → until	Qty
1	1432932	Oad by lever rh transmission kit depth 40 cm		1
2	1432933	Oad by lever lh transmission kit depth 40 cm		1
3	0240-05576	End cap lever oad by lever		1
4	1432930	Oad by lever reverse link + fixation		1
5	1433010	Inversion link + fixation		1
6	1433006	Transmission link + fixation d 40cm		1
7	1433502	Mb2 oad by lever rear wheel axle kit		1
8	9742-84304	Hollow screw + ball + spring		1
9	9028-40005	Oad by lever spring + cam		1
10	1433009	Driving cam completed		1
11	1435910	Oad By Lever Rh Transmission Kit D45Cm		1
12	1435911	Oad By Lever Lh Transmission Kit D 45Cm		1
13	1435912	Transmission Link + Fixation D 45Cm		1
14	1442034	Oad By Lever Brake Kit D 45 Cm		1
15	1534887	Rear wheel support for fixed axle <i>Replace 1431283</i>		1

Wheels For Oad By Lever

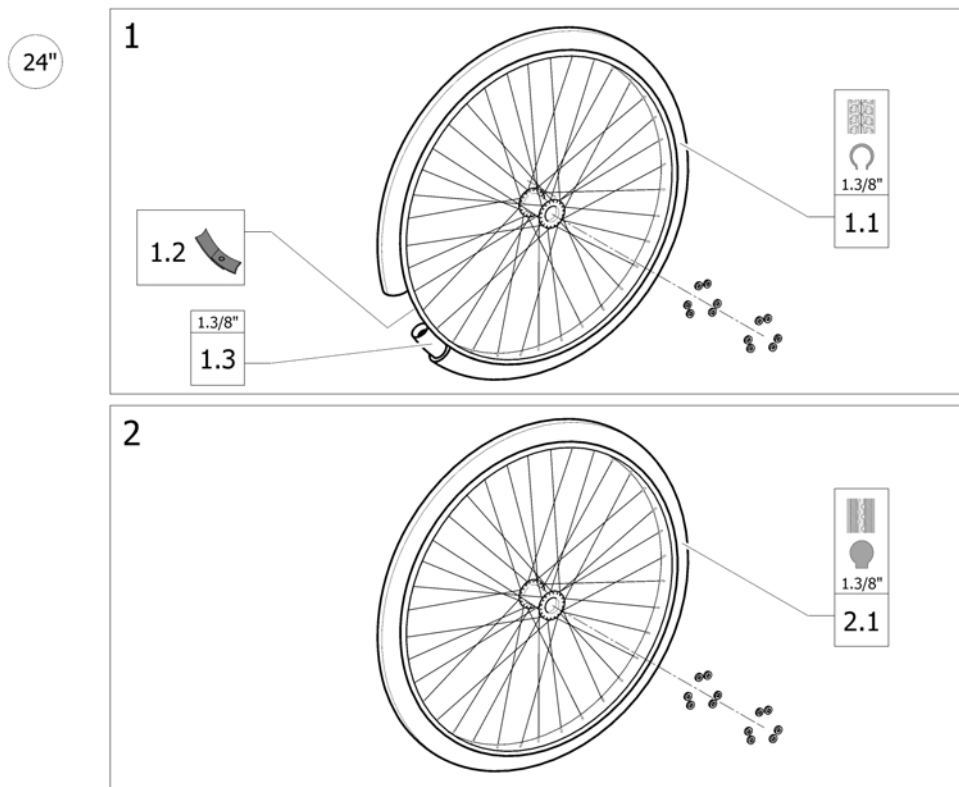

Pos.	Part number	Description	Valid from → until	Qty
1	1509380	24" x 1.3/8" oad spoked wheel + pneumatic tyre		2
1.1	0036-30001	24" x 1.3/8" pneumatic tyre		2
1.2	0036-40003	24" rim tape		2
1.3	0036-20001	24" x 1.3/8" inner tube		2
2	1509381	24" x 1.3/8" oad spoked wheel + solid tyre		2
2.1	0036-10013	24" x 1.3/8" solid tyre		2


Tipping tube

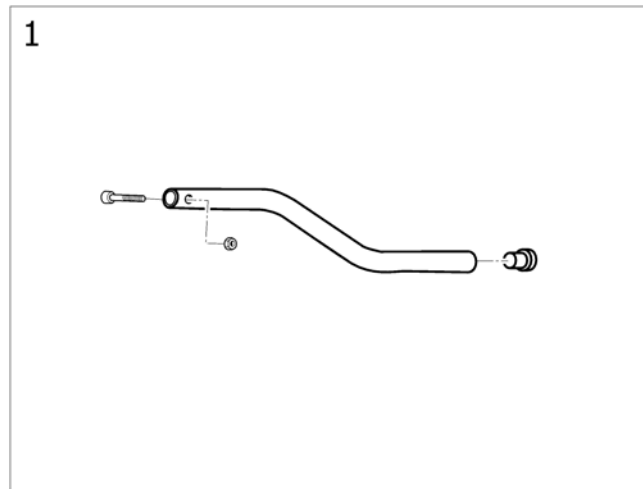

!SPS1496041


Pos.	Part number	Description	Valid from → until	Qty
1	0572-05143	Tipping Tube		1

Cane holder

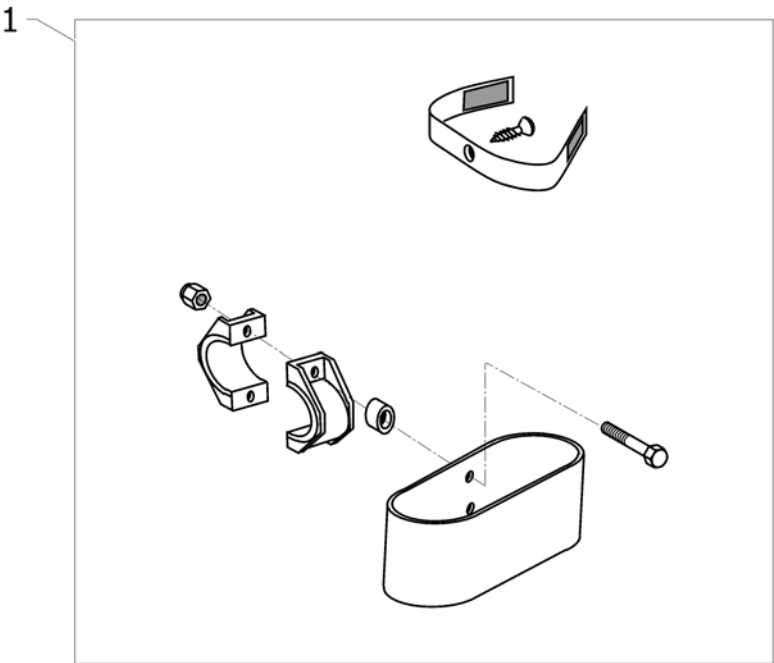

1095133401

Pos.	Part number	Description	Valid from → until	Qty
1	0572-02056	Cane Holder		1

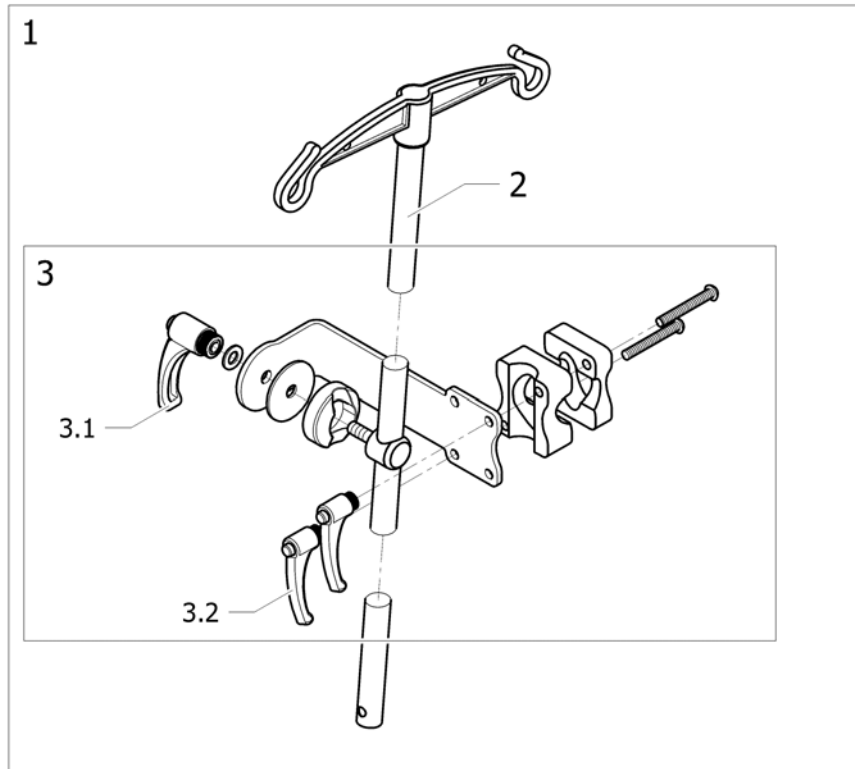Dripstand


ISPS1535040

Pos.	Part number	Description	Valid from → until	Qty
1	1534212	Dripstand mounted on back brace		1
2	2240-06160	Dripstand		1
3	1541969	Drip stand support on backbrace		1
3.1	0021-70011	Indexable lever 63 m8		1
3.2	1428547	Indexable Lever; Female M6		2

Transparent Tray




SW38	1
SW40.5	2
SW43	3
SW45.5	4
SW48	5
SW50.5	6

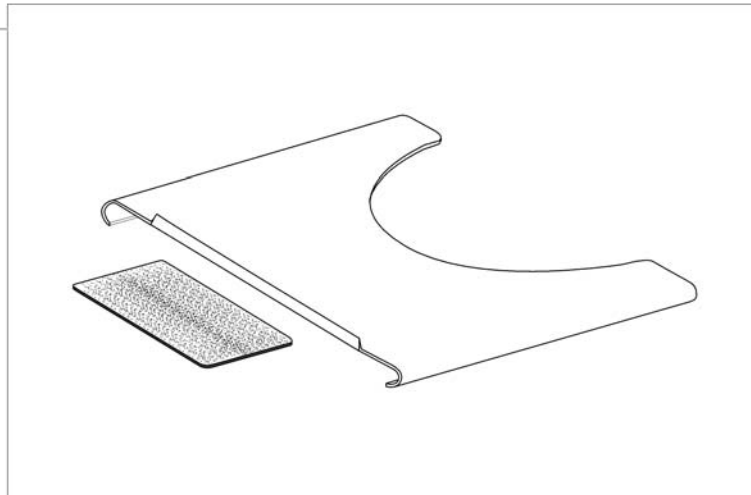

ISPS1496083

Pos.	Part number	Description	Valid from → until	Qty
1	1450921	Transparent Tray W 38Cm		1
2	1450922	Transparent Tray W 40,5Cm		1
3	1450923	Transparent Tray W 43Cm		1
4	1450924	Transparent Tray W 45,5Cm		1
5	1450925	Transparent Tray W 48Cm		1
6	1450926	Transparent Tray W 50,5Cm		1

Comfort Tray

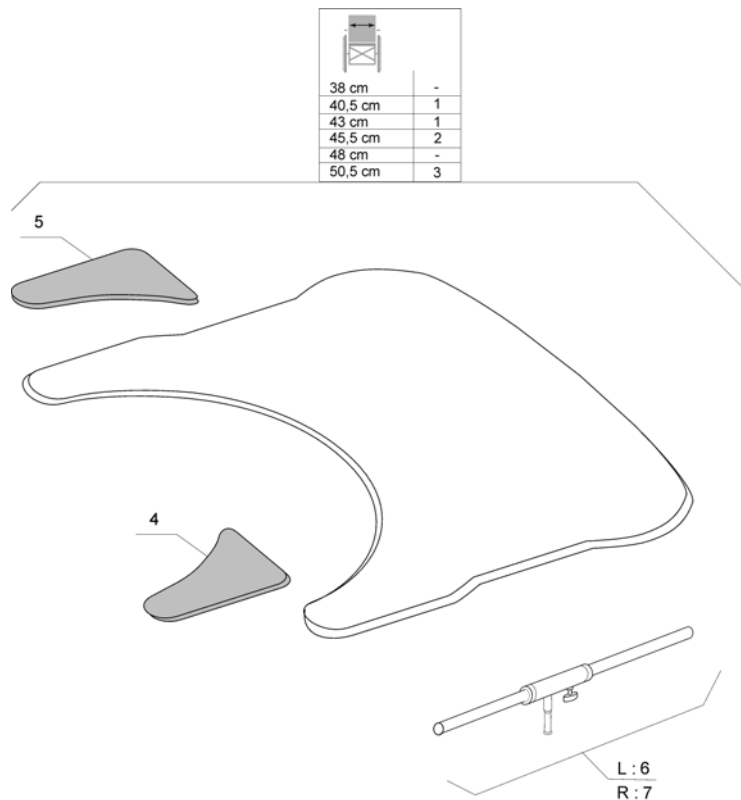

Pos.	Part number	Description	Valid from → until	Qty
1	5206727	Table Tray S, Cpl + Mounting Kit		1
2	5206726	Table Tray M, Cpl + Mounting Kit		1
3	5206725	Table Tray L, Cpl + Mounting Kit		1
4	5221346	Rh Elbowpad Table		1
5	5221354	Lh Elbowpad Table		1
6	5206648	Guide Tube Cpl, Left		1
7	5206649	Guide Tube Cpl, Right		1



Yes, you can.®



Belgium & Luxemburg:

Invacare nv, Autobaan 22, B-8210 Loppem
Tel: (32) (0)50 83 10 10, Fax: (32) (0)50 83 10 11
belgium@invacare.com

Danmark:

Invacare A/S, Sdr. Ringvej 37, DK-2605 Brøndby
Tel: (45) (0)36 90 00 00, Fax: (45) (0)36 90 00 01
denmark@invacare.com

Deutschland:

Invacare Aquatec GmbH, Alemannenstraße 10, D-88316 Isny
Tel: (49) (0)75 62 7 00 0, Fax: (49) (0)75 62 7 00 66
info@invacare-aquatec.com

European Distributor Organisation:

Invacare, Kleiststraße 49, D-32457 Porta Westfalica
Tel: (49) (0)57 31 754 540, Fax: (49) (0)57 31 754 541
edo@invacare.com

España:

Invacare SA, c/Areny s/n, Polígon Industrial de Celrà, E-17460 Celrà (Girona)
Tel: (34) (0)972 49 32 00, Fax: (34) (0)972 49 32 20
contactsp@invacare.com

France:

Invacare Poirier SAS, Route de St Roch, F-37230 Fondettes
Tel: (33) (0)2 47 62 64 66, Fax: (33) (0)2 47 42 12 24
contactfr@invacare.com

Ireland:

Invacare Ireland Ltd, Unit 5 Seatown Business Campus, Seatown Road, Swords, County Dublin - Ireland
Tel: (353) 1 810 7084, fax: (353) 1 810 7085
ireland@invacare.com

Italia:

Invacare Mecc San s.r.l., Via dei Pini 62, I-36016 Thiene (VI)
Tel: (39) 0445 38 00 59, Fax: (39) 0445 38 00 34
italia@invacare.com

Nederland:

Invacare BV, Celsiusstraat 46, NL-6716 BZ Ede
Tel: (31) (0)318 695 757, Fax: (31) (0)318 695 758
nederland@invacare.com, csede@invacare.com

Norge:

Invacare AS, Grensesvingen 9, Postboks 6230, Etterstad, N-0603 Oslo
Tel: (47) (0)22 57 95 00, Fax: (47) (0)22 57 95 01
norway@invacare.com, island@invacare.com

Österreich:

Mobitec Mobilitätshilfen GmbH, Herzog Odilostrasse 101, A-5310 Mondsee
Tel: (43) 6232 5535 0, Fax: (43) 6232 5535 4
office@mobitec-austria.com, austria@invacare.com

Portugal:

Invacare Lda, Rua Estrada Velha, 949, P-4465-784 Leça do Balio
Tel: (351) (0)225 1059 46147, Fax: (351) (0)225 1057 39
portugal@invacare.com

Sverige & Suomi:

Invacare AB, Fagerstagatan 9, S-163 91 Spånga
Tel: (46) (0)8 761 70 90, Fax: (46) (0)8 761 81 08
sweden@invacare.com, finland@invacare.com

Switzerland:

Mobitec Rehab AG, Benkenstrasse 260, CH-4108 Witterswil
Tel: (41) (0)61 487 70 80, Fax: (41) (0)61 487 70 81
office@mobitec-rehab.ch, switzerland@invacare.com

United Kingdom:

Invacare Limited, Pencoed Technology Park, Pencoed, Bridgend CF35 5HZ
Switchboard Tel: (44) (0)1656 776200, Fax: (44) (0)1656 776201
Customer services Tel: (44) (0)1656 776222, Fax: (44) (0)1656 776220
UK@invacare.com