

Spare Parts Catalogue

Version: 2015/09/10a EN



Invacare® *Action*®⁴ *NG*



Yes, you can.®

General Information

For any order of spare parts the following information is required:

- Product type*
- Product name & model*
- Serial Number*
- Part-Number*
- Part-Description*

Exploded view drawings and code numbers are not contractual and subject to modification, in relation to Invacare® continuous product improvement policy to meet end users needs.

The reproduction or use of this work, or of any substantial part of this work, in any form or by any electronic, mechanical or other means, including photocopying, scanning and recording, and in any information storage and retrieval system, including electronic transmission, is forbidden without the prior written permission of Invacare® International Sarl.



Yes, you can.®

TABLE OF CONTENTS

Information

Abbreviations	7
---------------------	---

Chassis

Chassis	8
Side frame	9
Folding System double cross brace	11

Seat

Seat	12
Seat Upholstery	13
Padded Seats Upholsteries	15
Adjustable Depth Seat Upholstery	16
Adjustable Seat Upholstery HD NL	17
Seat upholsteries cushions comfort Mistral	18
Anatomic Seat Adjustable	19
Toilet seat	20

Backrest

Backrest	21
----------------	----

Backrests

Backrests	22
Fixed Backrest 7° Height Push Handle	23
Additional Push Handle	24
Fixed Backrest	25
Fixed Backrest 350 mm Xtra	26
Fixed Backrest, Height Adjustable, Push Handle	27
Fold down backrest steel	28
Fold down backrest steel/aluminum	29
Adjustable Backrest From -6° To +24°	30
Recliner Backrest 0° To 30°, Ratchet	31
Recliner Backrest 0° To 30°, Gas Spring	32
Recliner Backrest 0° To 30°, Gas Spring, Adj Handle =>2011-12-31	33
Reclining/Folding backrest 0/30°, gas spring	34
Laguna backrests	35

Backrest upholsteries

Backrest upholsteries	36
Backrest upholstery Rea	37
Lumbar support padded back upholstery	38
Lumbar support padded back upholstery Bimaterial	40
Backrest Upholstery For Height Adjustable Backrest	42
Backrest Upholstery Bimaterial For Height Adjustable Backrest	43
Contour adjustable padded back upholstery	44
Contour adjustable Bimaterial padded back upholstery	46
Contour adjustable nylon type back upholstery	48
Contour adjustable Bimaterial nylon type back upholstery	50



Yes, you can.®

TABLE OF CONTENTS

Contour adjustable backrest upholstery Rea	52
Comfort upholstery	53
Mounting Hardware MatrX	55
Backrest upholstery MatrX	56

Legrests

Legrests	57
Hanger fixed 80°	58
Hanger fixed 80° =>2015.01.25	59
Assembly Legrest with footrest	60
Elevating legrest	61
Calf Pad	62
Amputee Legrest	63

Footrests

Footrests	64
Fixed footrest	65
Fixed footrest with shorter support	66
Angle adjustable footrest =>2010.09.21	67
Angle and depth adjustable footrest	68
Angle and depth adjustable footrest SH360 mm	70
Angle and depth adjustable footrest with shorter support	71
One piece footboard	72

Siderests

Siderests	73
Height Adjustable Armrest 1 Rio	74
T armrest n°1 Rio - Renouvable and height adjustable	75
Swing Away & Detachable Armrest 2	76
Height adjustable armrest n°3	78
Height adjustable armrest n°4	80
Hemi Dartex armpad n°4 multiposition standard	81
Mudguard Xtra	82

Front Wheels

Front Wheels	83
Forks	84
Forks supports adjustable	85
Front wheels	86
Front wheel extension kit	87

Rear Wheels

Rear Wheels	88
24" Standard Wheels	89
24" Heavy Duty Wheels	90
24" Standard Wheels accessories	91
22" Standard Wheels	92
22" Heavy Duty Wheels	93
22" Standard Wheels accessories	94
20" Standard Wheels	95



Yes, you can.®

TABLE OF CONTENTS

20" Standard Wheels accessories	96
24" / 22" Hub brake wheels	97
Quick release axle dia 12 Action	98
Spoke Guard	99
Rear Wheel Supports	100
12" Rear Wheels	101
Pump	102

Brakes

Brakes	103
Push Brakes : Standard And With Extension	104
Push brake Transit	105
Pull brakes : Standard And With Extension	106
Hub Brakes	107
Oad Wire Control Brake	108

Safety Options

Safety Options	109
Transportation attachment kit	110
Belts	111
Rear Anti Tipper and Tipping Tube	112
Reflectors	113

Seating Options

Seating Options	114
Cushions Nylon	115
Cushions Lite	116
Cushions Contour Visco	117
Cushions Image	118
Cushions Solution Xtra	119
Headrest	120
Neckrest	121
Cheek support	122
Soft pad headrest and adapter Matrix	123
Standard Headrest =>2011-06-30	124
Calf Strap	125
Heels strap	126
Back Brace standard + support	127
Back Brace type Rea	128

Propulsion Options

Propulsion Options	129
Dual handrim QR axle 2014.09.29 =>	130
Dual handrim QR axle => 2014.09.28	131
Hand Control For Oad By Lever	132
Oad By Lever	133
Wheels For Oad By Lever	134
Rear wheel transit supports for manual brakes	135
Rear wheel transit supports for hub brakes	136




Yes, you can.®

TABLE OF CONTENTS

Others Options & Accessories

Others Options & Accessories	137
Cane holder	138
Tipping tube	139
Dripstand	140
Transparent tray	141
Kit Tools	142



Abbreviations

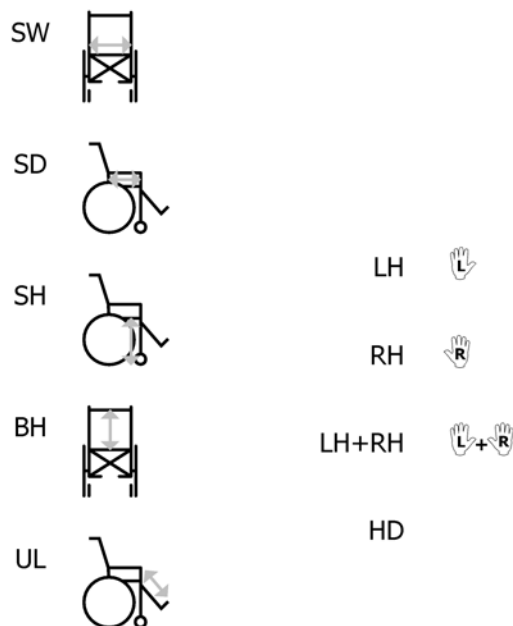

1895133721

Pos.	Part number	Description	Valid from → until	Qty
-	SW	Seat width (2 digits in cm) (3 digits in mm)		1
-	SD	Seat depth (2 digits in cm) (3 digits in mm)		1
-	SH	Seat height (2 digits in cm) (3 digits in mm)		1
-	BH	Backrest height (2 digits in cm) (3 digits in mm)		1
-	UL	Knee-to-heel height (2 digits in cm) (3 digits in mm)		1
-	LH	Left side (sitting in the wheelchair)		1
-	RH	Right side (sitting in the wheelchair)		1
-	LH+RH	An assembly of a left and a right part, referred to as "pair"		1
-	HD	Heavy duty version		1


Chassis

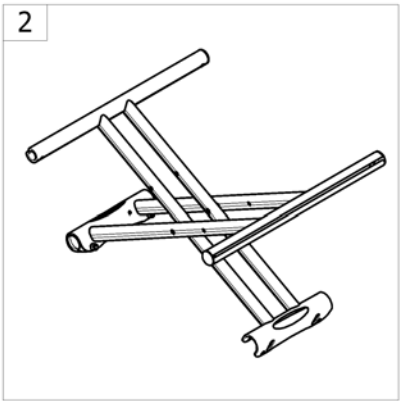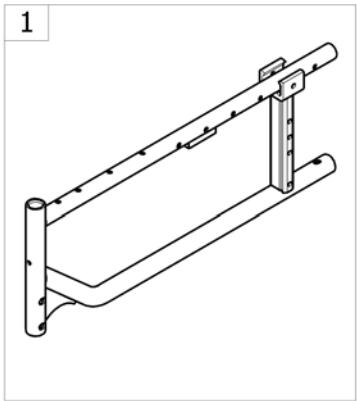

<SPACTION4NG10

Pos.	Part number	Description	Valid from → until	Qty
1	-	Side frame		1
2	-	Folding System double cross brace		1

Side frame

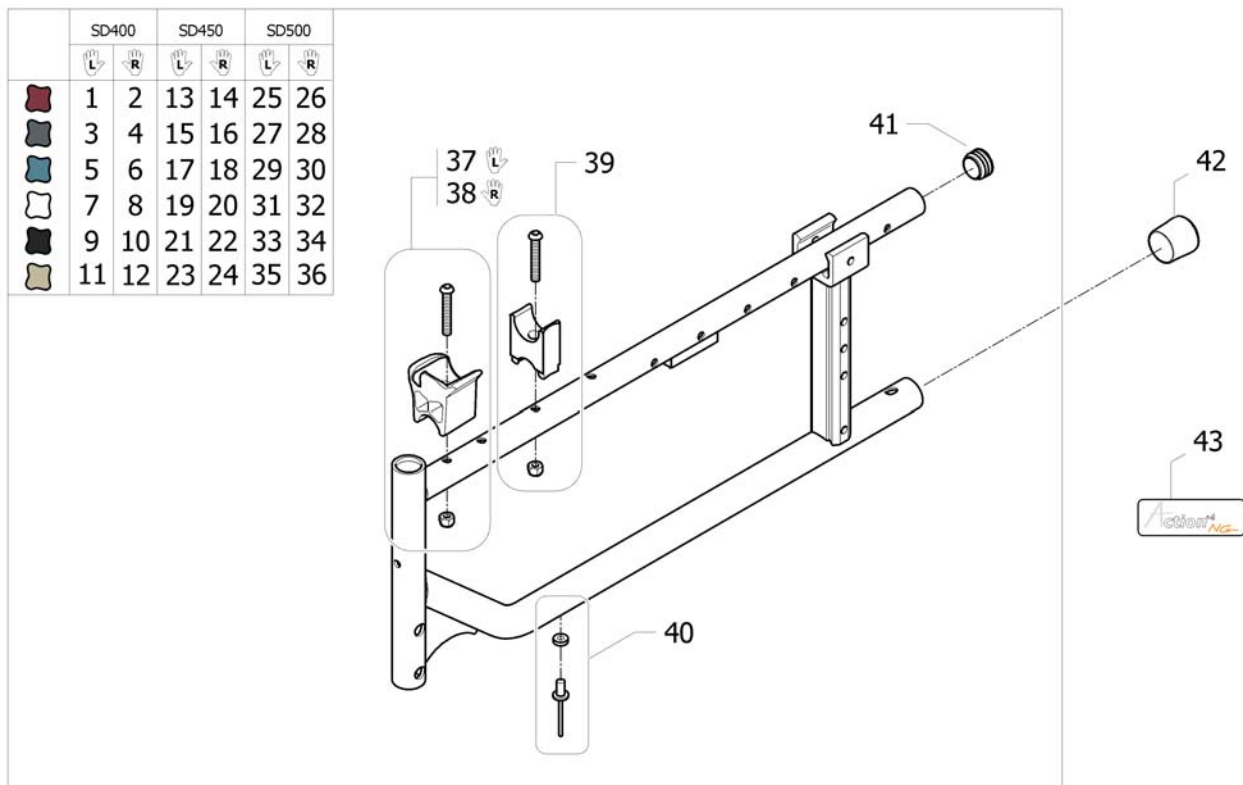


193515381

Pos.	Part number	Description	Valid from → until	Qty
1	1532974-0607	A4NG side frame Lh SD400 mm Dark Red		1
2	1532973-0607	A4NG side frame Rh SD400 mm Dark Red		1
3	1532974-0615	A4NG side frame Lh SD400 mm Slate Grey		1
4	1532973-0615	A4NG side frame Rh SD400 mm Slate Grey		1
5	1532974-0617	A4NG side frame Lh SD400 mm Ocean Blue		1
6	1532973-0617	A4NG side frame Rh SD400 mm Ocean Blue		1
7	1532974-0637	A4NG side frame Lh SD400 mm Pearl White		1
8	1532973-0637	A4NG side frame Rh SD400 mm Pearl White		1
9	1532974-0609	A4NG side frame Lh SD400 mm Anthracite		1
10	1532973-0609	A4NG side frame Rh SD400 mm Anthracite		1
11	1532974-0616	A4NG side frame Lh SD400 mm Ice Brown	→ 2011-12-31	1
12	1532973-0616	A4NG side frame Rh SD400 mm Ice Brown	→ 2011-12-31	1
13	1532976-0607	A4NG side frame Lh SD450 mm Dark Red		1
14	1532975-0607	A4NG side frame Rh SD450 mm Dark Red		1
15	1532976-0615	A4NG side frame Lh SD450 mm Slate Grey		1
16	1532975-0615	A4NG side frame Rh SD450 mm Slate Grey		1
17	1532976-0617	A4NG side frame Lh SD450 mm Ocean Blue		1
18	1532975-0617	A4NG side frame Rh SD450 mm Ocean Blue		1
19	1532976-0637	A4NG side frame Lh SD450 mm Pearl White		1
20	1532975-0637	A4NG side frame Rh SD450 mm Pearl White		1
21	1532976-0609	A4NG side frame Lh SD450 mm Anthracite		1
22	1532975-0609	A4NG side frame Rh SD450 mm Anthracite		1
23	1532976-0616	A4NG side frame Lh SD450 mm Ice Brown	→ 2011-12-31	1
24	1532975-0616	A4NG side frame Rh SD450 mm Ice Brown	→ 2011-12-31	1
25	1532978-0607	A4NG side frame Lh SD500 mm Dark Red		1
26	1532977-0607	A4NG side frame Rh SD500 mm Dark Red		1
27	1532978-0615	A4NG side frame Lh SD500 mm Slate Grey		1
28	1532977-0615	A4NG side frame Rh SD500 mm Slate Grey		1
29	1532978-0617	A4NG side frame Lh SD500 mm Ocean Blue		1
30	1532977-0617	A4NG side frame Rh SD500 mm Ocean Blue		1
31	1532978-0637	A4NG side frame Lh SD500 mm Pearl White		1

Side frame


Pos.	Part number	Description	Valid from → until	Qty
32	1532977-0637	A4NG side frame Rh SD500 mm Pearl White		1
33	1532978-0609	A4NG side frame Lh SD500 mm Anthracite		1
34	1532977-0609	A4NG side frame Rh SD500 mm Anthracite		1
35	1532978-0616	A4NG side frame Lh SD500 mm Ice Brown	→ 2011-12-31	1
36	1532977-0616	A4NG side frame Rh SD500 mm Ice Brown	→ 2011-12-31	1
37	1434128	Kit crossbar support clipable LH		1
38	1434127	Kit crossbar support clipable RH		1
39	0572-06097	Kit crossbar support in H for crossbar adjustable SW205/280 SD300/350 (4 extensions)		2
40	1492760	A4 Cross Brace Rivet & Washer Kit		2
41	1545553	End tube Ø22 reflector red <i>Replace 1418112</i>		2
42	5716276	UTV NDIG		2
43	1549114	Product name sticker Action4NG <i>Replace 1449467</i>		2

Folding System double cross brace

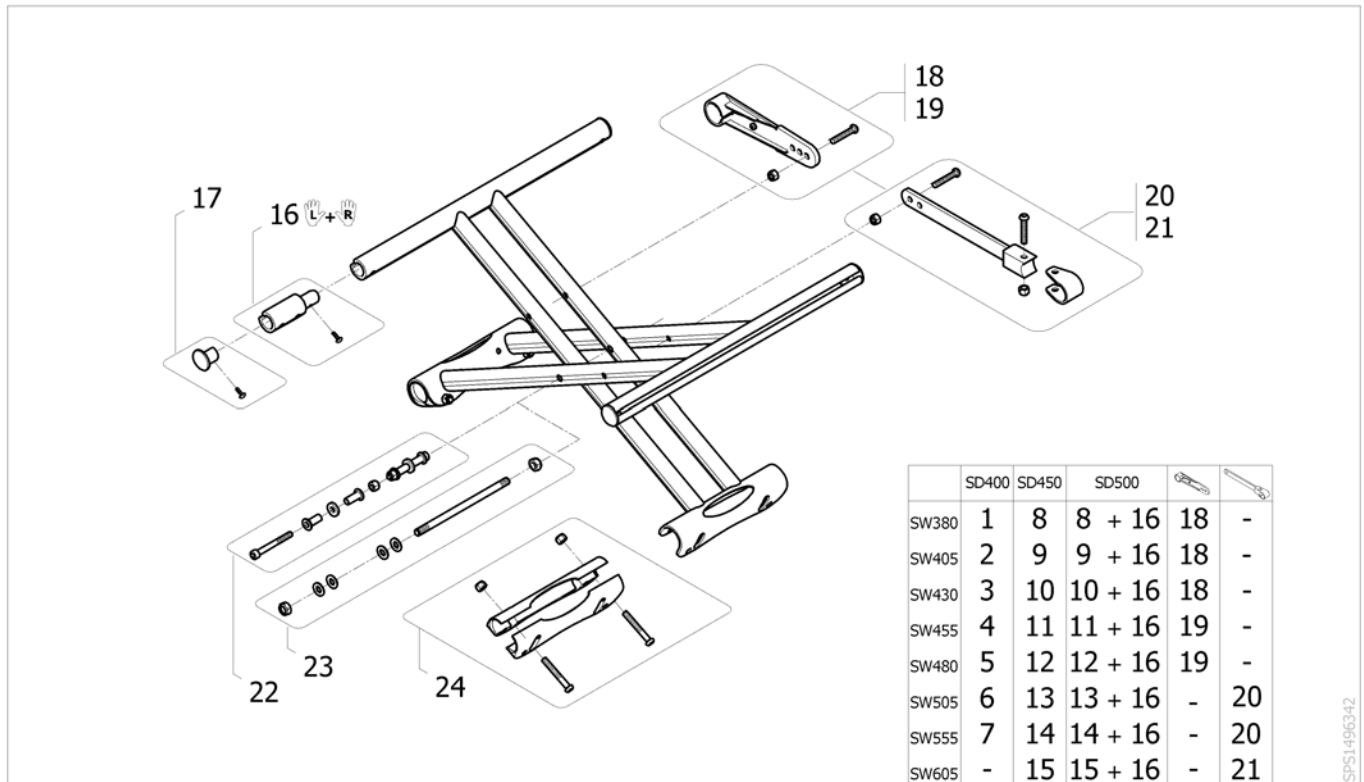

Pos.	Part number	Description	Valid from → until	Qty
1	1449961	A4 Cross Brace Complete (Pair) SW380 x SD400 mm		1
2	1449789	A4 Cross Brace Complete (Pair) SW405 x SD400 mm		1
3	1449790	A4 Cross Brace Complete (Pair) SW430 x SD400 mm		1
4	1449791	A4 Cross Brace Complete (Pair) SW455 x SD400 mm		1
5	1449792	A4 Cross Brace Complete (Pair) SW480 x SD400 mm		1
6	1488618	A4 Cross Brace Complete (Pair) SW505 x SD400 mm		1
7	1488620	A4 Cross Brace Complete Heavy(Pair) SW555 x SD400 mm		1
8	1449794	A4 Cross Brace Complete (Pair) SW380 x SD450 mm		1
9	1449795	A4 Cross Brace Complete (Pair) SW405 x SD450 mm		1
10	1449796	A4 Cross Brace Complete (Pair) SW430 x SD450 mm		1
11	1449797	A4 Cross Brace Complete (Pair) SW455 x SD450 mm		1
12	1449798	A4 Cross Brace Complete (Pair) SW480 x SD450 mm		1
13	1488619	A4 Cross Brace Complete (Pair) SW505 x SD450 mm		1
14	1488621	A4 Cross Brace Complete Heavy(Pair) SW555 x SD450 mm		1
15	1488622	A4 Cross Brace Complete Heavy(Pair) SW605 x SD450 mm		1
16	1510022	Kit crossbar extensions 5cm (x2)		1
17	1431307	Seat end cap kit		2
18	1451680	Cross Brace Strut SW380/430 mm		2
19	1451681	Cross Brace Strut SW455/505 mm		2
20	1488638	Cross Brace Strut SW555 mm		2
21	1451335	Cross Brace Strut SW605 mm		2
22	1451684	A4 Cross Brace Axle Kit		1
23	1538158	A4 Cross Brace Axle HD Kit		1
24	1451679	A4 Cross Brace Sleeve		2

Seat

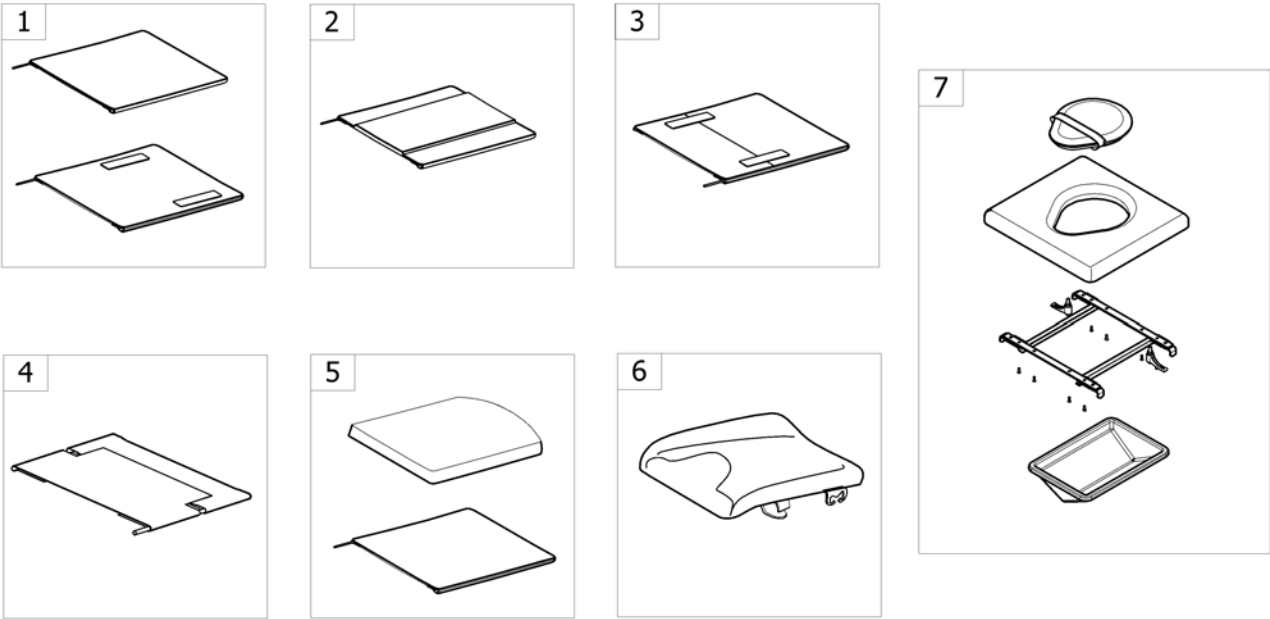


<SPCACTION4NG20


Pos.	Part number	Description	Valid from → until	Qty
1	-	Seat Upholstery		1
2	-	Padded Seats Upholsteries		1
3	-	Adjustable Depth Seat Upholstery		1
4	-	Adjustable Seat Upholstery HD NL		1
5	-	Seat upholstery cushions comfort Mistral		1
6	-	Anatomic Seat Ajustable		1
7	-	Toilet seat		1

Seat Upholstery

		Action 3&4NG	
		SD400	SD450
SW380	1	7	
SW405	2	8	
SW430	3	9	
SW455	4	10	
SW480	5	11	
SW505	6	12	

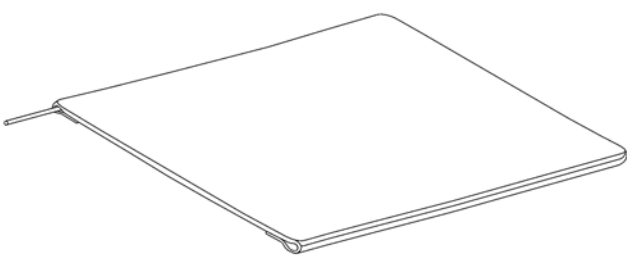

		Action 3NG		Action 4NG		
		SD400	SD450	SD400	SD450	SD500
SW380	13	19	13	19	25	
SW405	14	20	14	20	26	
SW430	15	21	15	21	27	
SW455	16	22	16	22	28	
SW480	17	23	17	23	29	
SW505	18	24	18	24	30	
SW555	-	-	-	-	31	
SW605	-	-	-	-	32	

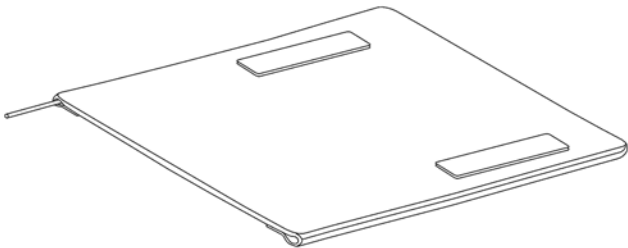


Pos.	Part number	Description	Valid from → until	Qty
1	1428840	Basic Seat Upholstery SW380 x SD400 mm		1
2	1428841	Basic Seat Upholstery SW405 x SD400 mm		1
3	1428842	Basic Seat Upholstery SW430 x SD400 mm		1
4	1428843	Basic Seat Upholstery SW455 x SD400 mm		1
5	1428844	Basic Seat Upholstery SW480 x SD400 mm		1
6	1428845	Basic Seat Upholstery SW505 x SD400 mm		1
7	1428846	Basic Seat Upholstery SW380 x SD450 mm		1
8	1428847	Basic Seat Upholstery SW405 x SD450 mm		1
9	1428848	Basic Seat Upholstery SW430 x SD450 mm		1
10	1428849	Basic Seat Upholstery SW455 x SD450 mm		1
11	1428850	Basic Seat Upholstery SW480 x SD450 mm		1
12	1428851	Basic Seat Upholstery SW505 x SD450 mm		1
13	1428852	Standard Seat Upholstery SW380 x SD400 mm		1
14	1428853	Standard Seat Upholstery SW405 x SD400 mm		1
15	1428854	Standard Seat Upholstery SW430 x SD400 mm		1
16	1428855	Standard Seat Upholstery SW455 x SD400 mm		1
17	1428856	Standard Seat Upholstery SW480 x SD400 mm		1
18	1428857	Standard Seat Upholstery SW505 x SD400 mm		1
19	1428858	Standard Seat Upholstery SW380 x SD450 mm		1
20	1428859	Standard Seat Upholstery SW405 x SD450 mm		1
21	1428860	Standard Seat Upholstery SW430 x SD450 mm		1
22	1428861	Standard Seat Upholstery SW455 x SD450 mm		1
23	1428862	Standard Seat Upholstery SW480 x SD450 mm		1
24	1428863	Standard Seat Upholstery SW505 x SD450 mm		1
25	1553738	Standard Seat Upholstery SW380 x SD500 mm		1
26	1553739	Standard Seat Upholstery SW405 x SD500 mm		1
27	1522806	Standard Seat Upholstery SW430 x SD500 mm		1
28	1522822	Standard Seat Upholstery SW455 x SD500 mm		1
29	1522823	Standard Seat Upholstery SW480 x SD500 mm		1
30	1510023	Standard Seat Upholstery SW505 x SD500 mm		1
31	1510024	Standard Seat Upholstery SW555 x SD500 mm		1

Seat Upholstery

Pos.	Part number	Description	Valid from → until	Qty
32	1510025	Standard Seat Upholstery SW605 x SD500 mm		1

Padded Seats Upholsteries

	Action 3NG		Action 4NG	
	SD400	SD450	SD400	SD450
SW380	1	8	1	8
SW405	2	9	2	9
SW430	3	10	3	10
SW455	4	11	4	11
SW480	5	12	5	12
SW505	6	13	6	13
SW555	-	-	7	14
SW605	-	-	-	15

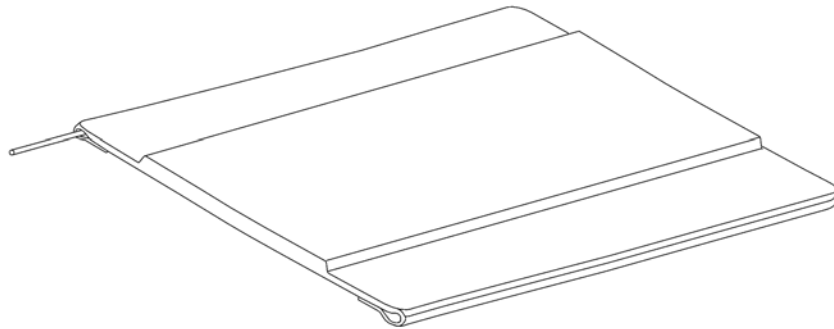


ISPS1496256

Pos.	Part number	Description	Valid from → until	Qty
1	1428864	Padded Seat Upholstery SW380 x SD400 mm		1
2	1428865	Padded Seat Upholstery SW405 x SD400 mm		1
3	1428866	Padded Seat Upholstery SW430 x SD400 mm		1
4	1428867	Padded Seat Upholstery SW455 x SD400 mm		1
5	1428868	Padded Seat Upholstery SW480 x SD400 mm		1
6	1428869	Padded Seat Upholstery SW505 x SD400 mm		1
7	1488394	Reinforced Padded Seat Upholstery SW555 x SD 400 mm		1
8	1428870	Padded Seat Upholstery SW380 x SD450 mm		1
9	1428871	Padded Seat Upholstery SW405 x SD450 mm		1
10	1428872	Padded Seat Upholstery SW430 x SD450 mm		1
11	1428873	Padded Seat Upholstery SW455 x SD450 mm		1
12	1428874	Padded Seat Upholstery SW480 x SD450 mm		1
13	1428875	Padded Seat Upholstery SW505 x SD450 mm		1
14	1488396	Reinforced Padded Seat Upholstery SW555 mm x SD450 mm		1
15	1488397	Reinforced Padded Seat Upholstery SW605 mm x SD450 mm		1


Adjustable Depth Seat Upholstery

	Action 3NG		Action 4NG	
	SD400>SD450		SD400>SD500	
SW380	1	-	1	-
SW405	2	-	2	-
SW430	3	-	3	9
SW455	4	-	4	10
SW480	5	-	5	11
SW505	6	-	6	12
SW555	-	-	7	13
SW605	-	-	8	14

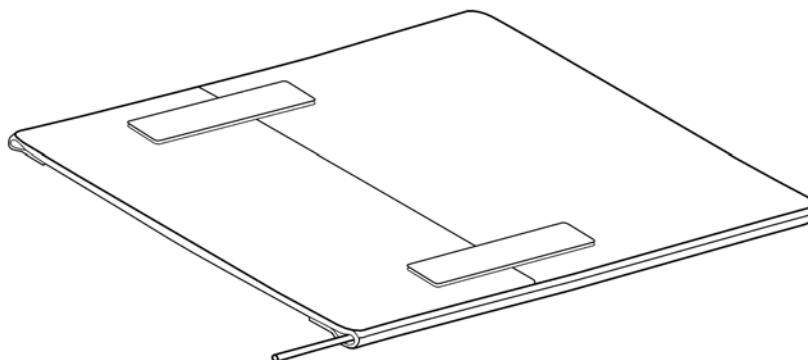


ISPS1497377

Pos.	Part number	Description	Valid from → until	Qty
1	1437580	Adjustable Depth Seat Upholstery SW380 x SD400 to 450 mm		1
2	1437581	Adjustable Depth Seat Upholstery SW405 x SD400 to 450 mm		1
3	1437582	Adjustable Depth Seat Upholstery SW430 x SD400 to 450 mm		1
4	1437583	Adjustable Depth Seat Upholstery SW455 x SD400 to 450 mm		1
5	1437584	Adjustable Depth Seat Upholstery SW480 x SD400 to 450 mm		1
6	1437585	Adjustable Depth Seat Upholstery SW505 x SD400 to 450 mm		1
7	1496089	Adjustable Depth Seat Upholstery SW555 x SD400 to 450 mm		1
8	1496090	Adjustable Depth Seat Upholstery SW605 x SD400 to 450 mm		1
9	1548717	Adjustable Depth Seat Upholstery SW430 x SD450 to 500 mm		1
10	1548718	Adjustable Depth Seat Upholstery SW455 x SD450 to 500 mm		1
11	1548719	Adjustable Depth Seat Upholstery SW480 x SD450 to 500 mm		1
12	1548720	Adjustable Depth Seat Upholstery SW505 x SD450 to 500 mm		1
13	1548722	Adjustable Depth Seat Upholstery SW555 x SD450 to 500 mm		1
14	1548723	Adjustable Depth Seat Upholstery SW605 x SD450 to 500 mm		1

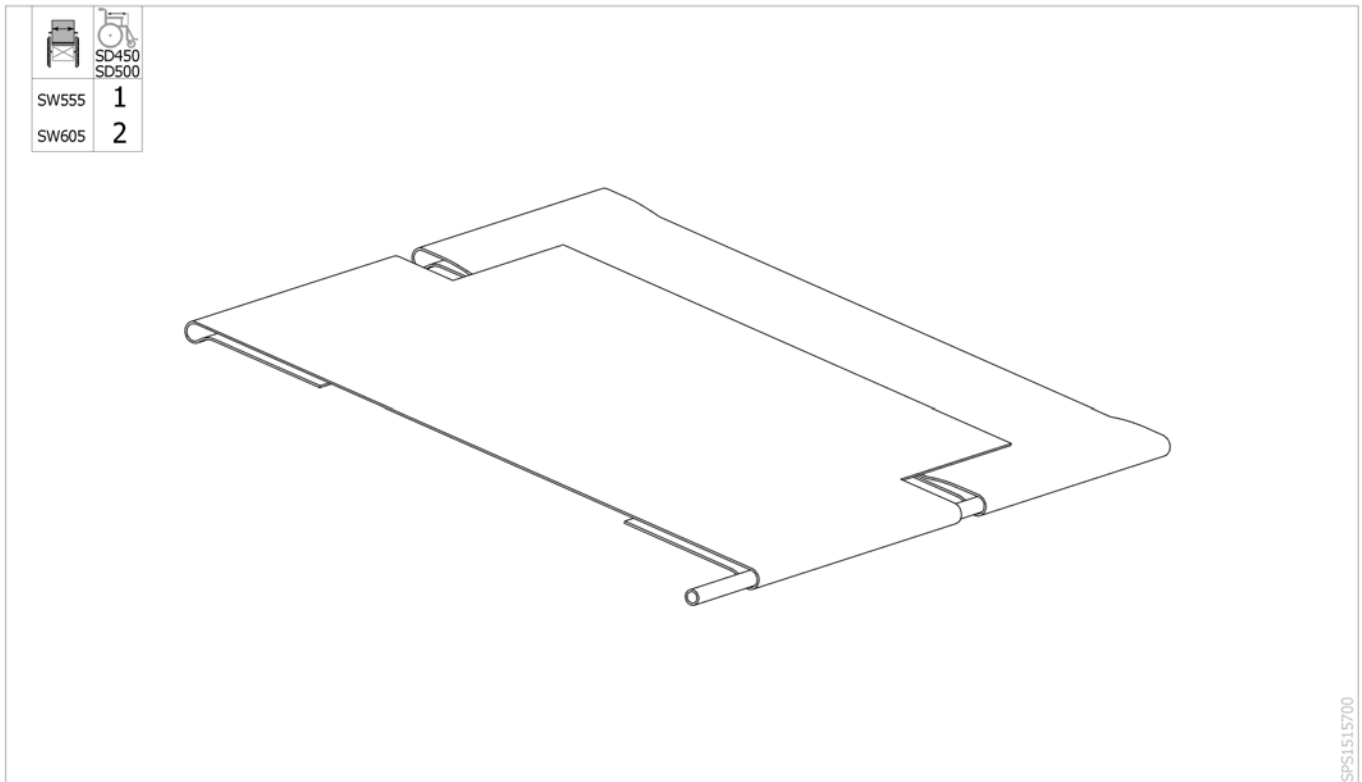Adjustable Seat Upholstery HD NL

Pos.	Part number	Description	Valid from → until	Qty
1	1515707	Adjustable seat upholstery HD NL SW555 x SD500 mm		1
2	1515708	Adjustable seat upholstery HD NL SW605 x SD500 mm		1


Seat upholsteries cushions comfort Mistral

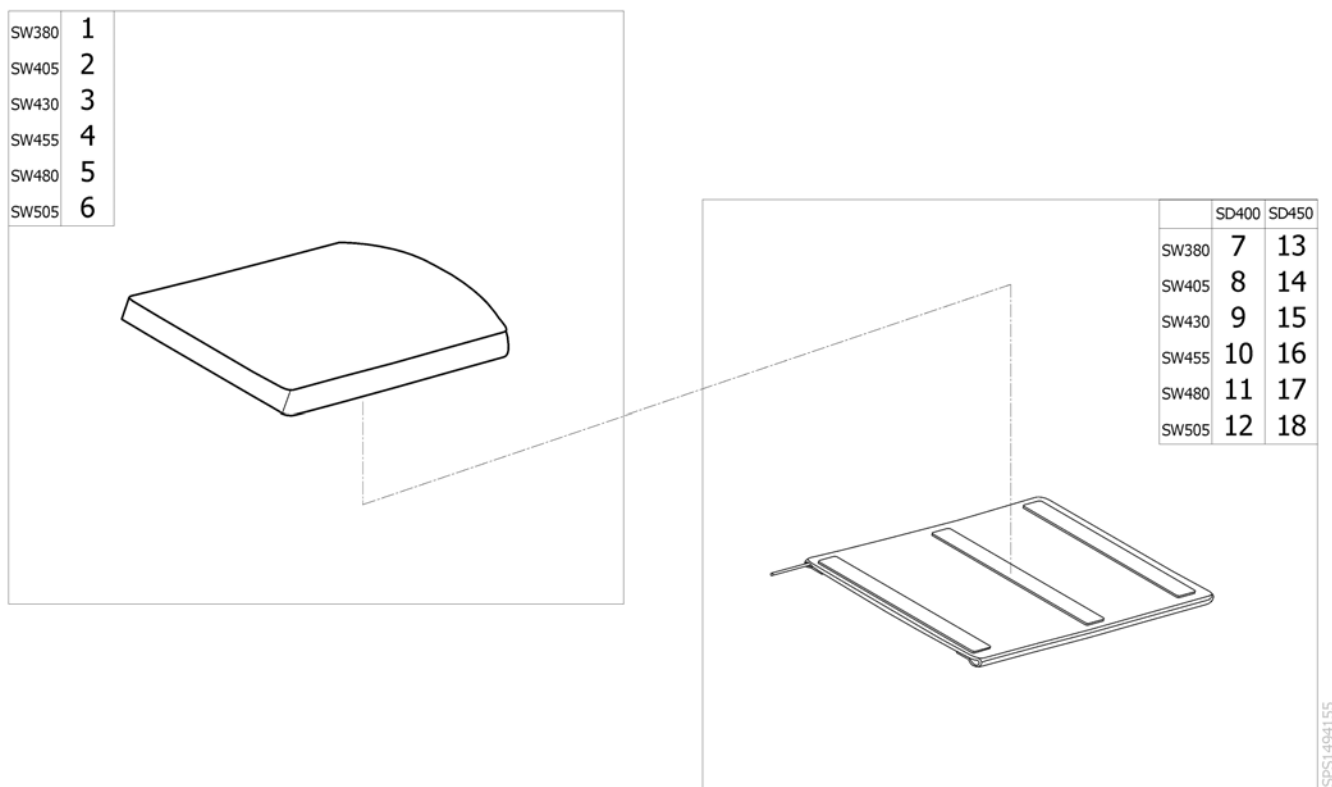

Pos.	Part number	Description	Valid from → until	Qty
1	1450115-TR26	Cushion Mistral 2 layer SW380 x SD450 mm Dartex Black TR26		1
2	1450116-TR26	Cushion Mistral 2 layer SW405 x SD450 mm Dartex Black TR26		1
3	1450117-TR26	Cushion Mistral 2 layer SW430 x SD450 mm Dartex Black TR26		1
4	1450118-TR26	Cushion Mistral 2 layer SW455 x SD450 mm Dartex Black TR26		1
5	1450119-TR26	Cushion Mistral 2 layer SW480 x SD450 mm Dartex Black TR26		1
6	1450120-TR26	Cushion Mistral 2 layer SW505 x SD450 mm Dartex Black TR26		1
7	1450067	Comfort Seat Upholstery SW380 x SD400 mm		1
8	1450068	Comfort Seat Upholstery SW405 x SD400 mm		1
9	1450069	Comfort Seat Upholstery SW430 x SD400 mm		1
10	1450070	Comfort Seat Upholstery SW455 x SD400 mm		1
11	1450071	Comfort Seat Upholstery SW480 x SD400 mm		1
12	1450072	Comfort Seat Upholstery SW505 x SD400 mm		1
13	1450123	Comfort Seat Upholstery SW380 x SD450 mm		1
14	1440918	Comfort Seat Upholstery SW405 x SD450 mm		1
15	1440919	Comfort Seat Upholstery SW430 x SD450 mm		1
16	1440920	Comfort Seat Upholstery SW455 x SD450 mm		1
17	1450124	Comfort Seat Upholstery SW480 x SD450 mm		1
18	1440921	Comfort Seat Upholstery SW505 x SD450 mm		1

Invacare® Action 4 NG | Seat



Anatomic Seat Adjustable

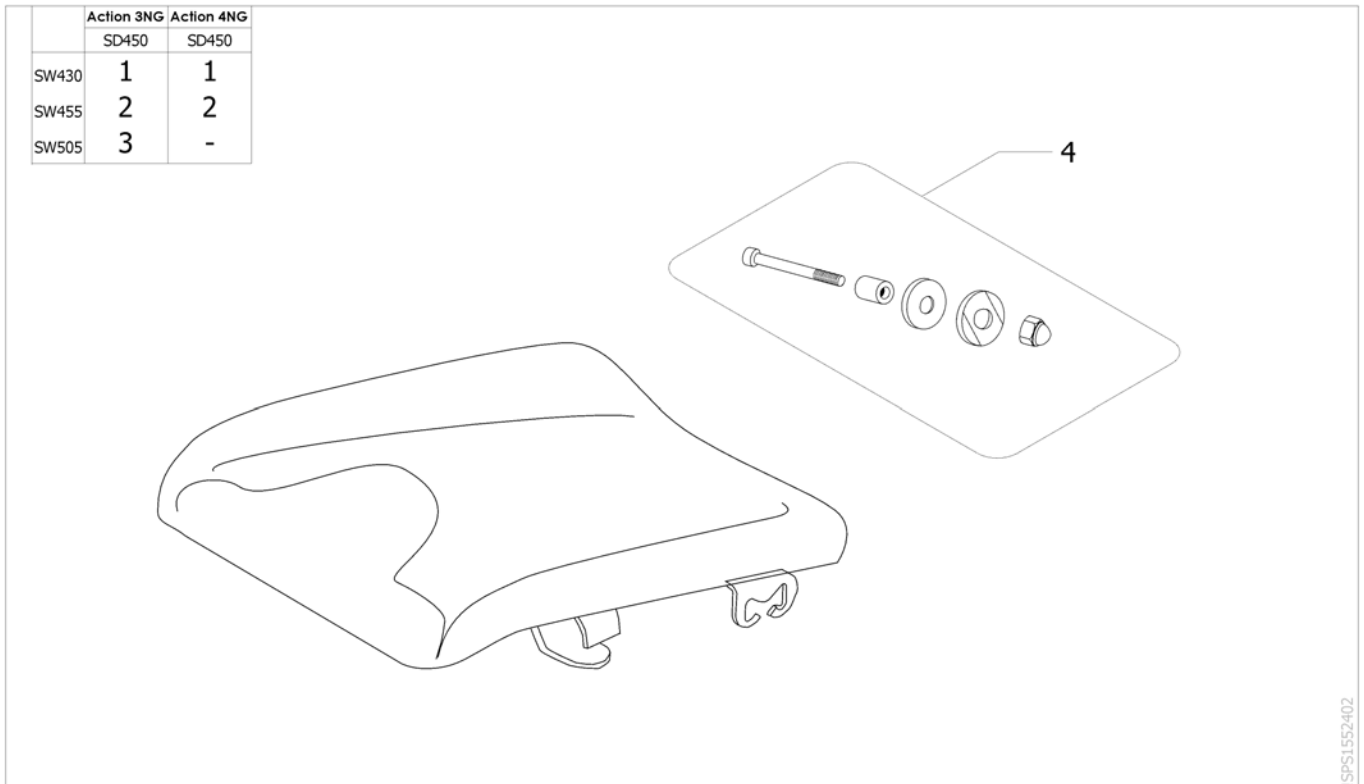

Pos.	Part number	Description	Valid from → until	Qty
1	1485159	Anatomic Seat Adjustable SW430 mm		1
2	1485160	Anatomic Seat Adjustable SW455 mm		1
3	1485161	Anatomic Seat Adjustable SW505 mm		1
4	1485535	Seat Frame Fixation		1

Toilet seat

	Action 3NG		Action 4NG		Action 1NG	Action 2NG
	SD400	SD450	SD400	SD450	SD400	SD400
SW380	1	4	1	-	-	1
SW405	1	4	1	-	1	1
SW430	2	-	2	-	2	2
SW455	2	-	2	-	2	2
SW480	3	-	3	-	3	3
SW505	3	-	-	-	3	-

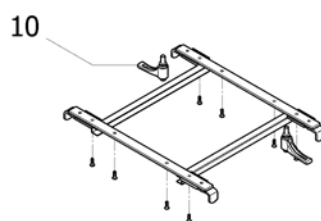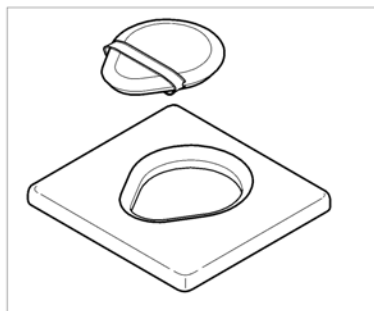

	Action 3NG		Action 4NG		Action 1NG	Action 2NG
	SD400	SD450	SD400	SD450	SD400	SD400
SW380	5	8	5	-	-	5
SW405	5	8	5	-	5	5
SW430	6	-	6	-	6	6
SW455	6	-	6	-	6	6
SW480	7	-	7	-	7	7
SW505	7	-	-	-	7	-


ISP51548795

Pos.	Part number	Description	Valid from → until	Qty
1	AP1543295	Kit toilet seat Action NG SD400 x SW380-410 mm		1
2	AP1543296	Kit toilet seat Action NG SD400 x SW430-455 mm		1
3	AP1543297	Kit toilet seat Action NG SD400 x SW480-510 mm		1
4	AP1543298	Kit toilet seat Action NG SD450 x SW380-405 mm		1
5	1548372	Kit toilet seat cushions Action NG SD400 SW380-410 mm		1
6	1548373	Kit toilet seat cushions Action NG SD400 SW430-455 mm		1
7	1548374	Kit toilet seat cushions Action NG SD400 SW480-510 mm		1
8	1548375	Kit toilet seat cushions Action NG SD450 SW380-405 mm		1
9	1548378	Toilet seat bowl Action NG SD400 mm		1
10	0021-70002	Indexable lever 63mm M8x16		2

Backrest

Pos.	Part number	Description	Valid from → until	Qty
1	-	Backrests		1
2	-	Backrest upholsteries		1


Backrests

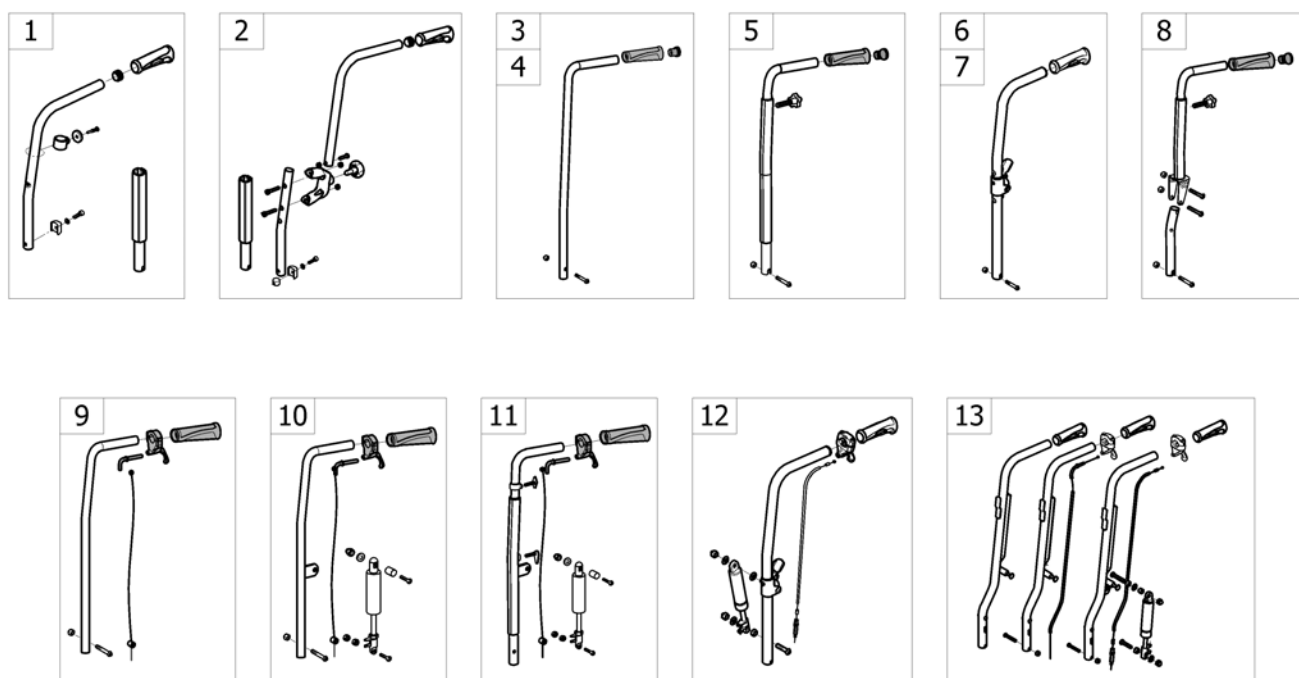

ISPS1574808

Pos.	Part number	Description	Valid from → until	Qty
1	-	Fixed Backrest 7° Height Push Handle [CK_0300]		1
2	-	Additional Push Handle [CK_0310]		1
3	-	Fixed Backrest [CK_0311/CK_0312]		1
4	-	Fixed Backrest 350 mm Xtra [CK_0315]		1
5	-	Fixed Backrest, Height Adjustable, Push Handle [CK_0314]		1
6	-	Fold down backrest steel [HD+CK_0341]		1
7	-	Fold down backrest steel/aluminum [CK_0341/CK_0342]		1
8	-	Adjustable Backrest From -6° To +24° [CK_0316]		1
9	-	Recliner Backrest 0° To 30°, Ratchet [CK_0320/CK_0330]		2
10	-	Recliner Backrest 0° To 30°, Gaz Spring [CK_0333/CK_0334]		1
11	-	Recliner Backrest 0° To 30°, Gaz Spring, Adj Handle =>2011-12-31 [CK_0340]		1
12	-	Reclining/Folding backrest 0/30°, gaz spring [CK_0345]		1
13	-	Laguna backrests [CK_0313/CK_0343/CK_0344]		1


Fixed Backrest 7° Height Push Handle

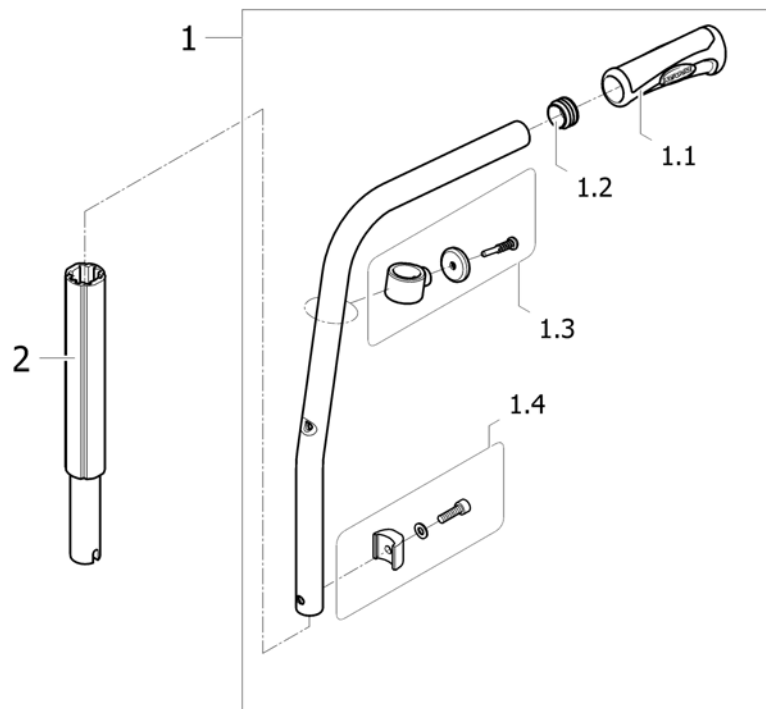



ISPS1538129

Pos.	Part number	Description	Valid from → until	Qty
1	1449802	A4 Backrest 7°/380 mm To 460 mm Cane Kit [CE_0395][CK_0300]		1
1.1	1435420	Handle Invacare for backrest Ø20mm		2
1.2	1418112	Crossbrace back cap Ø22x15mm		2
1.3	1451816	A4 Fixed Backrest Upholstery Support Kit		2
1.4	1449803	A4 Fixed Backrest Fixation Kit		2
2	1538204	Backrest extension kit, removable		2

Additional Push Handle

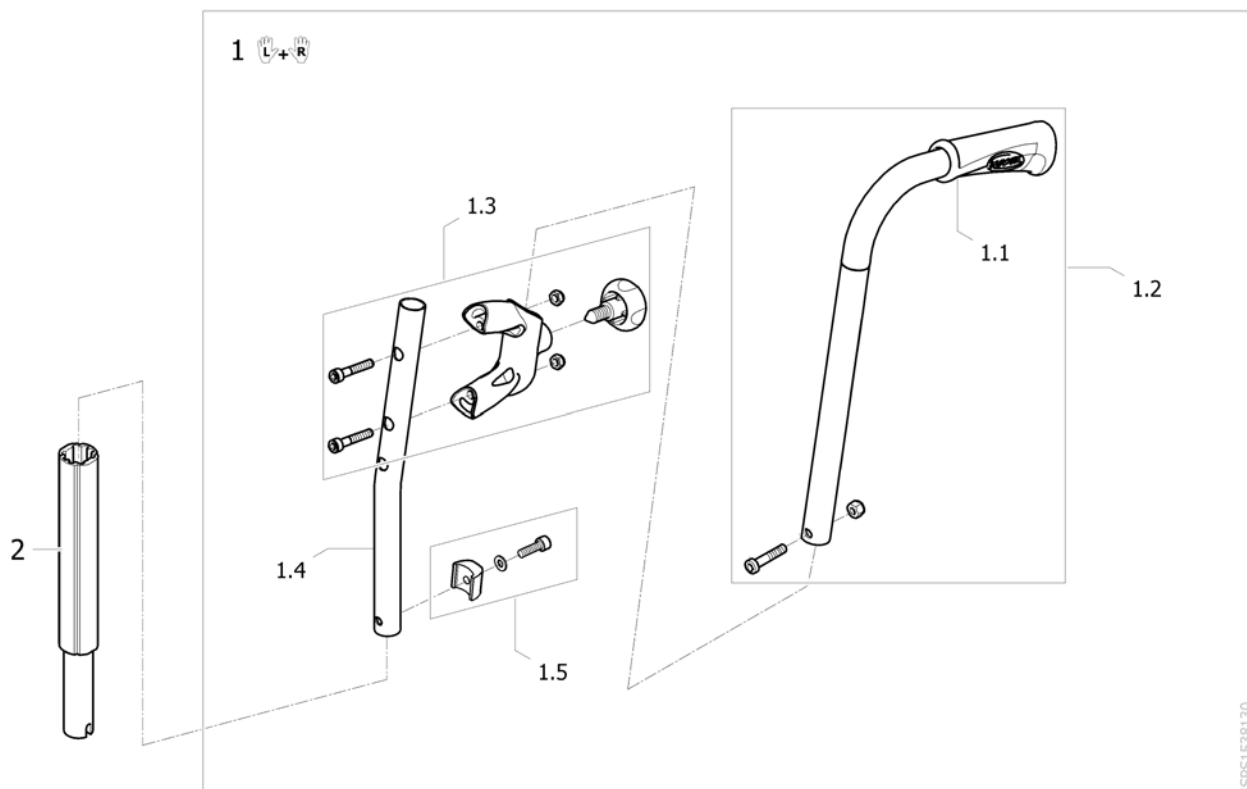


Pos.	Part number	Description	Valid from → until	Qty
1	1449804	Additional Push Handle Set (Pair) [CE_0396][CK_0310]		1
1.1	1435420	Handle Invacare for backrest Ø20mm		1
1.2	1450574	Kit rearsset pushhandle tube		1
1.3	1451818	Kit rearsset pushhandle support		1
1.4	SP1574655	Backrest tube for additional push handle Replace 1515698	2014-08-26 →	1
1.5	1449803	A4 Fixed Backrest Fixation Kit		1
2	1538204	Backrest extension kit, removable		1

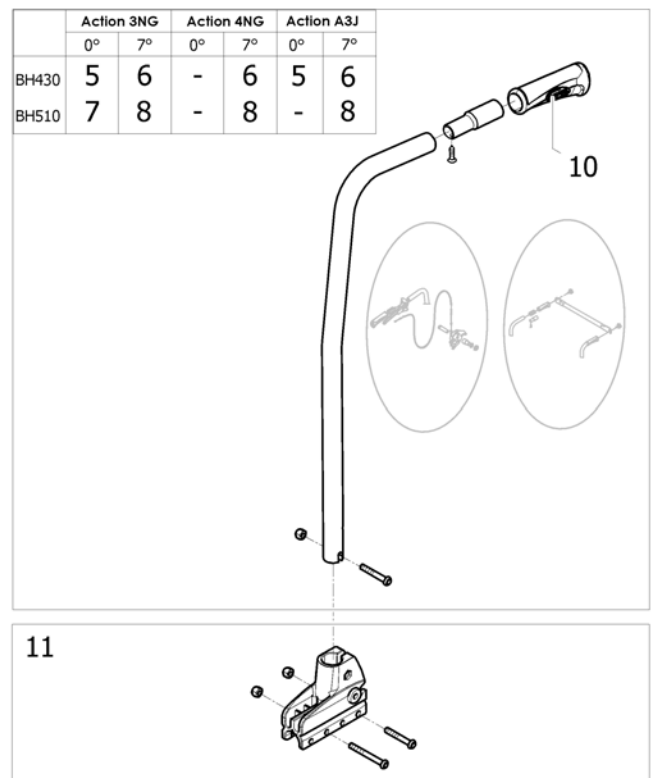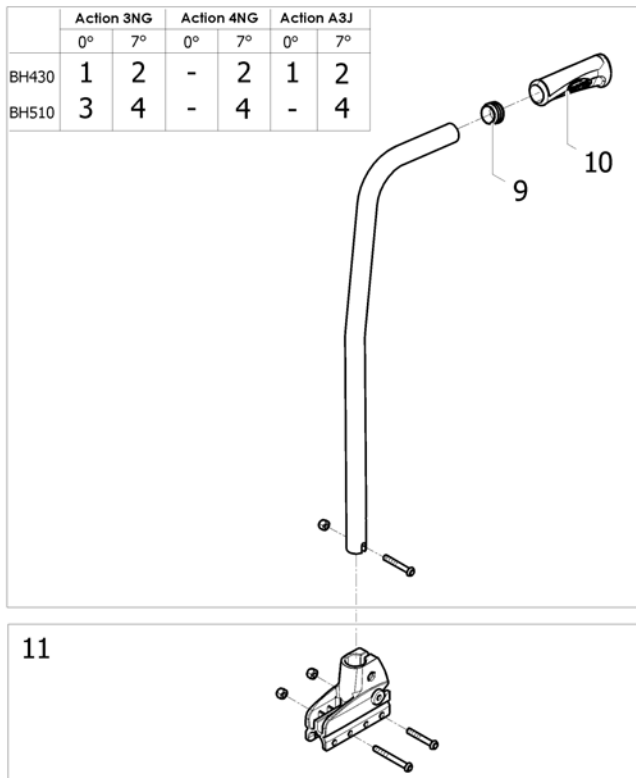Fixed Backrest

Pos.	Part number	Description	Valid from → until	Qty
1	1430844	Backrest tube fixed 0°/430 mm (Unite) [CE_0380][CN_0370]		2
2	1430846	Backrest tube fixed 7°/430 mm (Unité) [CE_0410][CK_0311][CN_0380]		2
3	1430845	Backrest 0°/510 mm Cane Kit (Unité) [CE_0390]		2
4	1430847	Backrest tube fixed 7°/510 mm (Unité) [CE_0420][CK_0312][CN_0381]		2
5	SP1581636	Backrest tube fixed With insert 0°/430 mm (Unite)		2
6	SP1581637	Backrest tube fixed With insert 7°/430 mm (Unite)		2
7	SP1581638	Backrest tube fixed With insert 0°/510 mm (Unite)		2
8	SP1581639	Backrest tube fixed With insert 7°/510 mm (Unite)		2
9	1418112	Crossbrace back cap Ø22x15mm		2
10	1425473	Handle Invacare for backrest Ø22mm Replace 1435421		2
11	1430848	Fixed Backrest Support Kit		2



Fixed Backrest 350 mm Xtra

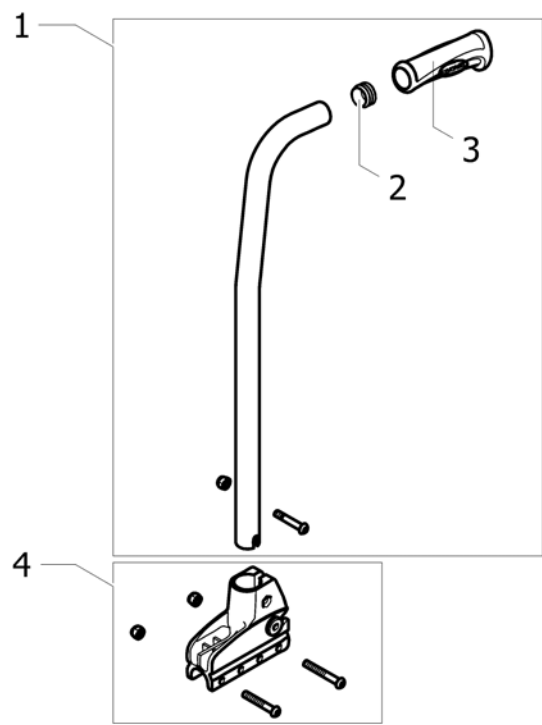

199133338

Pos.	Part number	Description	Valid from → until	Qty
1	1485911	Fixed Backrest Cane 350 mm (pair) <i>[CE_0428][CK_0315]</i>		1
2	1418112	Crossbrace back cap Ø22x15mm		2
3	1425473	Handle Invacare for backrest Ø22mm <i>Replace 1435421</i>		2
4	1430848	Fixed Backrest Support Kit		2


Fixed Backrest, Height Adjustable, Push Handle

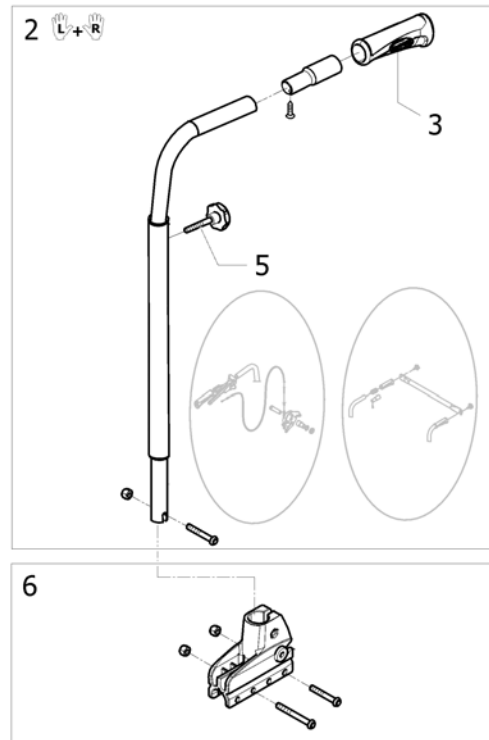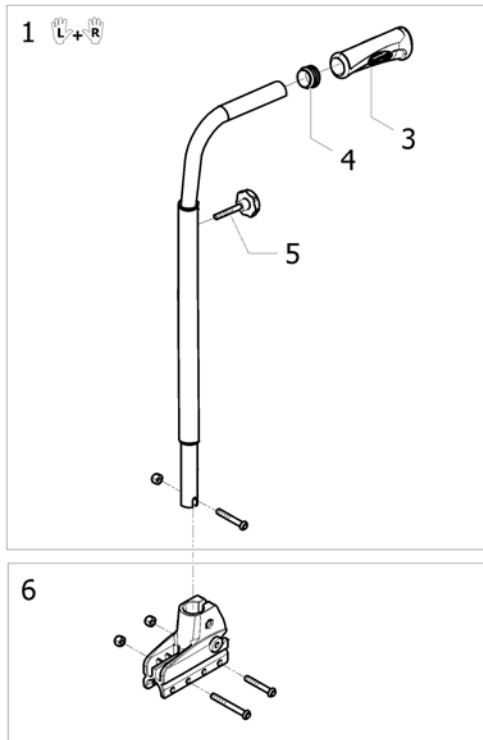

ISPS1494157


Pos.	Part number	Description	Valid from → until	Qty
1	SP1586293	Backrest tube adjustable Push Handle BH430/580 mm (pair) <i>[CE_0425][CK_0314][CN_0425] - Replace 1428641-0601</i>		1
2	SP1586294	Backrest tube adjustable Push Handle With insert BH430/580 mm (Pair) <i>[CE_0425][CK_0314][CN_0425] if headrest/backbrace or hub brake - Replace SP1581640</i>		1
3	1425473	Handle Invacare for backrest Ø22mm <i>Replace 1435421</i>		2
4	1418112	Crossbrace back cap Ø22x15mm		2
5	0021-60029	Button Ø32 mm M6x16		2
6	1430848	Fixed Backrest Support Kit		2

Fold down backrest steel

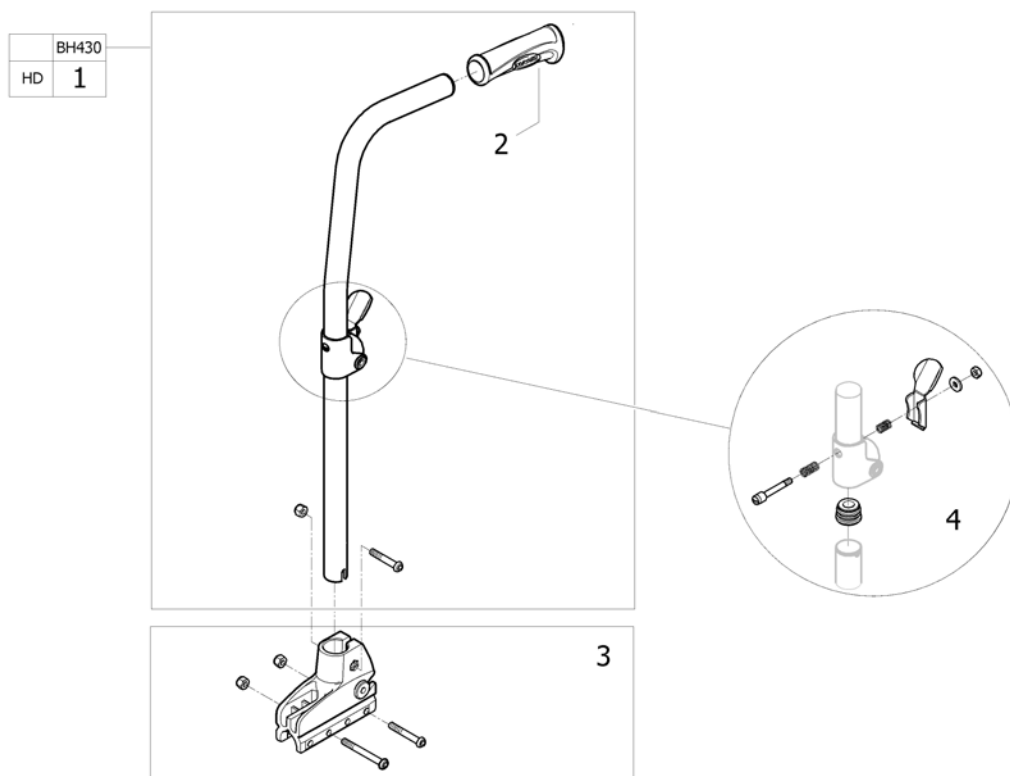

ISPS1548803



Pos.	Part number	Description	Valid from → until	Qty
1	1548793	Kit folding backrest cane Steel BH430 mm [HD+CE_0455]/[HD+CK_0341]		2
2	1425473	Handle Invacare for backrest Ø22mm		2
3	1430848	Fixed Backrest Support Kit		2
4	1551064	Locking system for folding backrest	2011-07-01 →	2

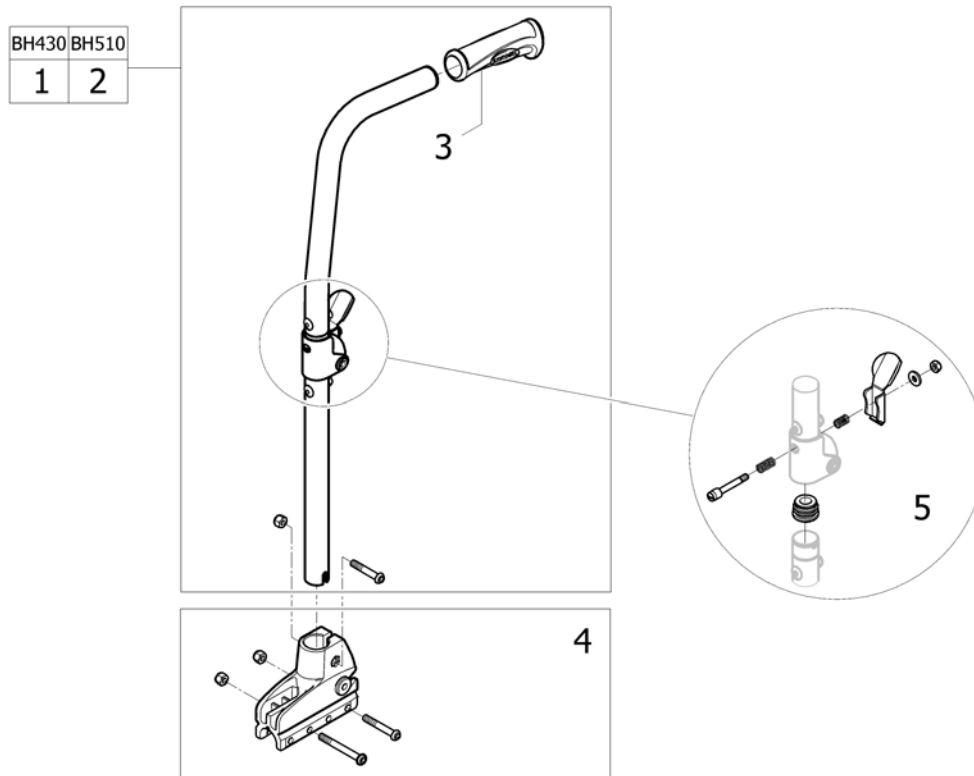Fold down backrest steel/alu

ISPS155318 ~ sps1545638_II

Pos.	Part number	Description	Valid from → until	Qty
1	1434492	Backrest tube Folding steel/alu BH 430 mm (Unité) [CE_0455][CK_0341][CN_0400][CZ_0455]		2
2	1430852	Backrest tube Folding steel/alu BH 510 mm (Unité) [CE_0460][CK_0342][CN_0401]		2
3	1425473	Handle Invacare for backrest Ø22mm		2
4	1430848	Fixed Backrest Support Kit		2
5	1551064	Locking system for folding backrest	2011-07-01 →	2

Adjustable Backrest From -6° To +24°

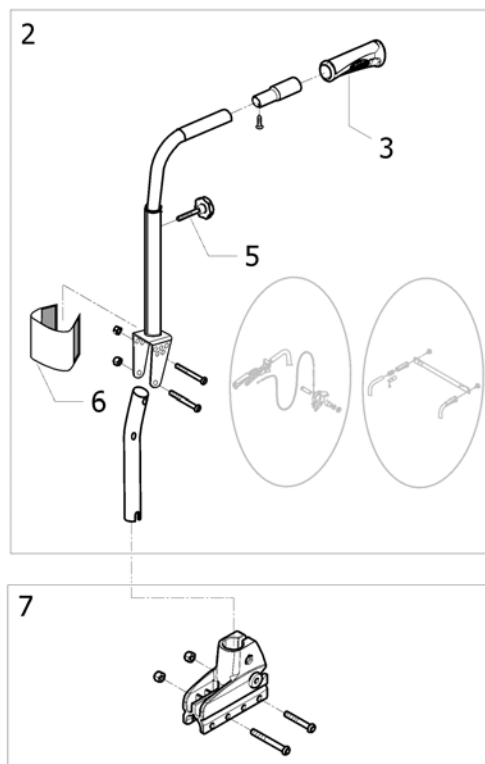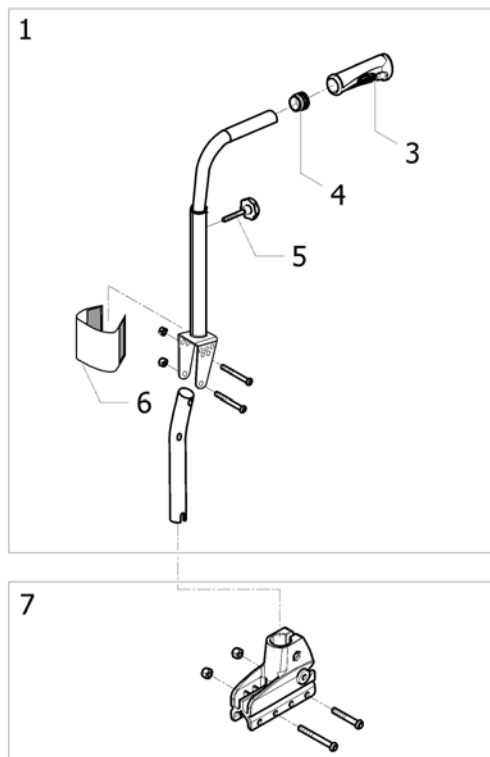


ISPS1494158

Pos.	Part number	Description	Valid from → until	Qty
1	1443021	-6°/+24° Backrest Cane Kit (Unite) [CE_0430][CK_0316]		2
2	SP1581641	-6°/+24° Backrest Cane Kit With insert (Unité)		2
3	1425473	Handle Invacare for backrest Ø22mm Replace 1435421		2
4	1418112	Crossbrace back cap Ø22x15mm		2
5	0021-60029	Button Ø32 mm M6x16		2
6	1442721	Protector Of Backrest Articulation		2
7	1430848	Fixed Backrest Support Kit		2

Recliner Backrest 0° To 30°, Ratchet


	Action 3NG Action 4NG	Action A3J
BH430	1	1
BH510	2	-

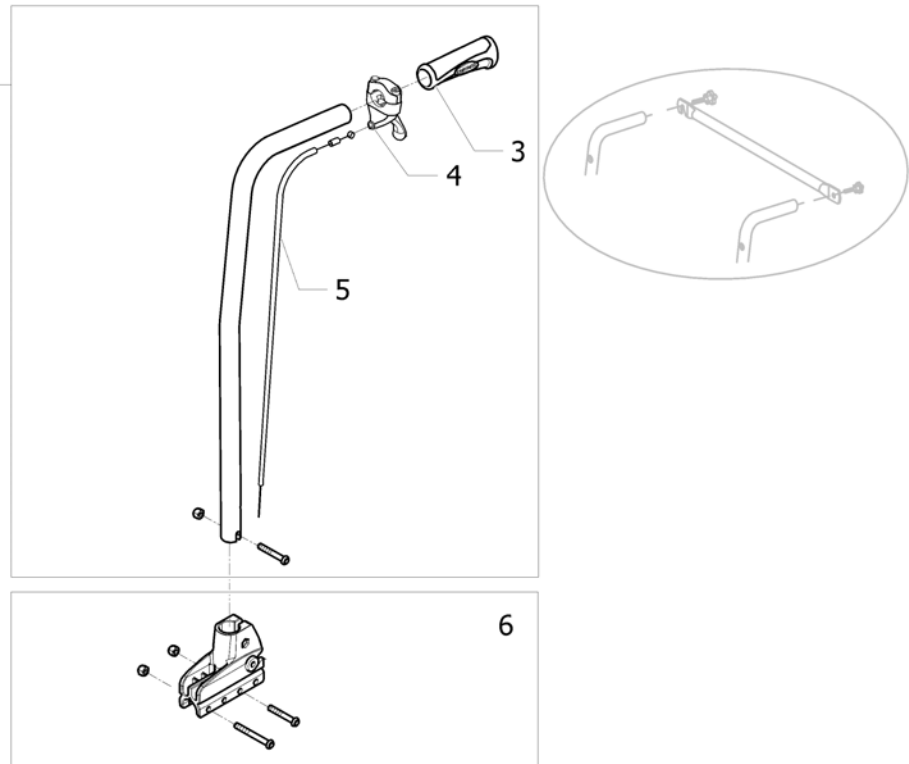

ISPS1494169


Pos.	Part number	Description	Valid from → until	Qty
1	1434491	Backrest tube recliner ratchet 0° to 30° SH350-370-390-410-430 (Unite) [CE_0435][CK_0320][CN_0390]		2
2	1430849	Reclining Backrest Cane BH 510 mm Complete (Unité) [CE_0440][CK_0330]		2
3	1425473	Handle Invacare for backrest Ø22mm Replace 1435421		2
4	1433747	Recliner backrest trigger		2
5	1430851	Recliner backrest cable Kit		2
6	1430850	Recliner Backrest Support Kit		2

Recliner Backrest 0° To 30°, Gaz Spring

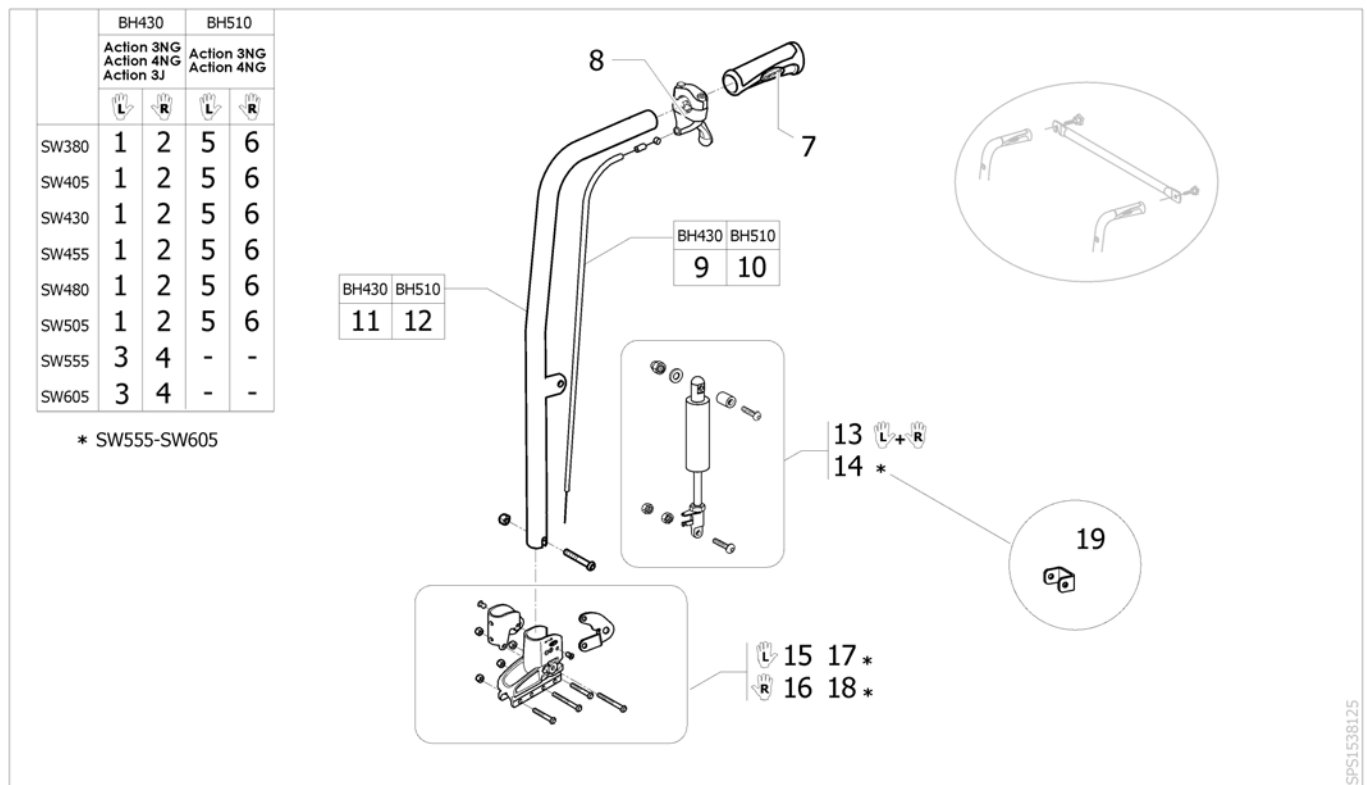

Pos.	Part number	Description	Valid from → until	Qty
1	1442539	Backrest tube Lh recliner Gas Strut BH 430 mm (Unité) [CE_0445][CK_0333][CN_0395]		1
2	1442538	Backrest tube Rh recliner Gas Strut BH 430 mm (Unité) [CE_0445][CK_0333][CN_0395]		1
3	1493416	Reclining Lh Kit Gas Strut Backrest BH 430 mm HD [HD+CE_0445][HD+CK_0333]		1
4	1493415	Reclining Rh Kit Gas Strut Backrest BH 430 mm HD [HD+CE_0445][HD+CK_0333]		1
5	1442540	Reclining Lh Kit Gas Strut Backrest BH 510 mm [CE_0450][CK_0334]		1
6	1442534	Reclining Rh Kit Gas Strut Backrest BH 510 mm [CE_0450][CK_0334]		1
7	1425473	Handle Invacare for backrest Ø22mm Replace 1435421		2
8	1433747	Recliner backrest trigger		2
9	1440937	Recliner Gas Cable BH430 mm		2
10	1485871	Recliner Gas Cable BH510 mm		2
11	1484055-0699	Recliner Gas Tube BH430 mm		2
12	1484054-0699	Recliner Gas Tube BH510 mm		2
13	1532595	Gas Strut Kit Standard Backrest	2011-12-02 →	2
13	1528909	Gas Strut Kit Standard Backrest (pair)	2009-09-16 → 2011-12-01	1
14	1532596	Gas Strut Kit Standard Backrest Hd		2
15	1442513	Lh Recliner Gas Backrest Support Kit		2
16	1442507	Rh Recliner Gas Backrest Support Kit		2
17	1493418	Lh Recliner Gas Backrest Support Kit Hd		1
18	1493417	Rh Recliner Gas Backrest Support Kit Hd		1
19	1450992-0699	Head gas spring reinforcement		2


Recliner Backrest 0° To 30°, Gaz Spring, Adj Handle =>2011-12-31

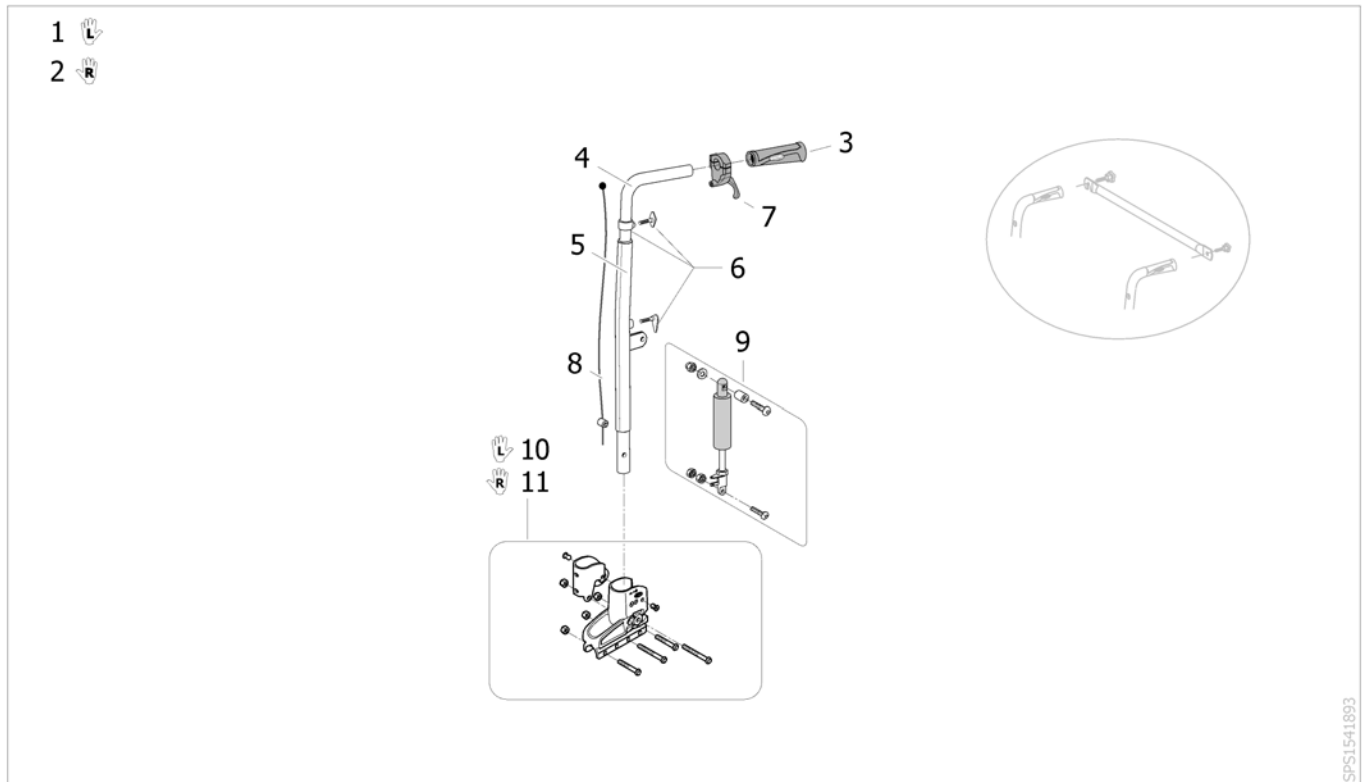


ISPS1541893

Pos.	Part number	Description	Valid from → until	Qty
1	1541894	Reclining Lh Kit Gas Strut Adjustable Backrest [CE_0451][CK_0340]	→ 2011-12-31	1
2	1541895	Reclining Rh Kit Gas Strut Adjustable Backrest [CE_0451][CK_0340]	→ 2011-12-31	1
3	1425473	Handle Invacare for backrest Ø22mm Replace 1435421	→ 2011-12-31	1
4	1448986-0010	Adjustable Push Handle Recliner Gas A4	→ 2011-12-31	1
5	1448985-0601	Adjustable Recliner Cane A4	→ 2011-12-31	1
6	1451819	Adjustable Backrest Push Handle Kit	→ 2011-12-31	1
7	1433747	Recliner backrest trigger	→ 2011-12-31	1
8	1451765	Adjustable Recliner Gas Cable Kit	→ 2011-12-31	1
9	1532595	Gas Strut Kit Standard Backrest	2011-12-02 → 2011-12-31	2
9	1528909	Gas Strut Kit Standard Backrest (pair)	2009-09-16 → 2011-12-01	1
10	1442513	Lh Recliner Gas Backrest Support Kit	→ 2011-12-31	1
11	1442507	Rh Recliner Gas Backrest Support Kit	→ 2011-12-31	1

Reclining/Folding backrest 0/30°, gaz spring

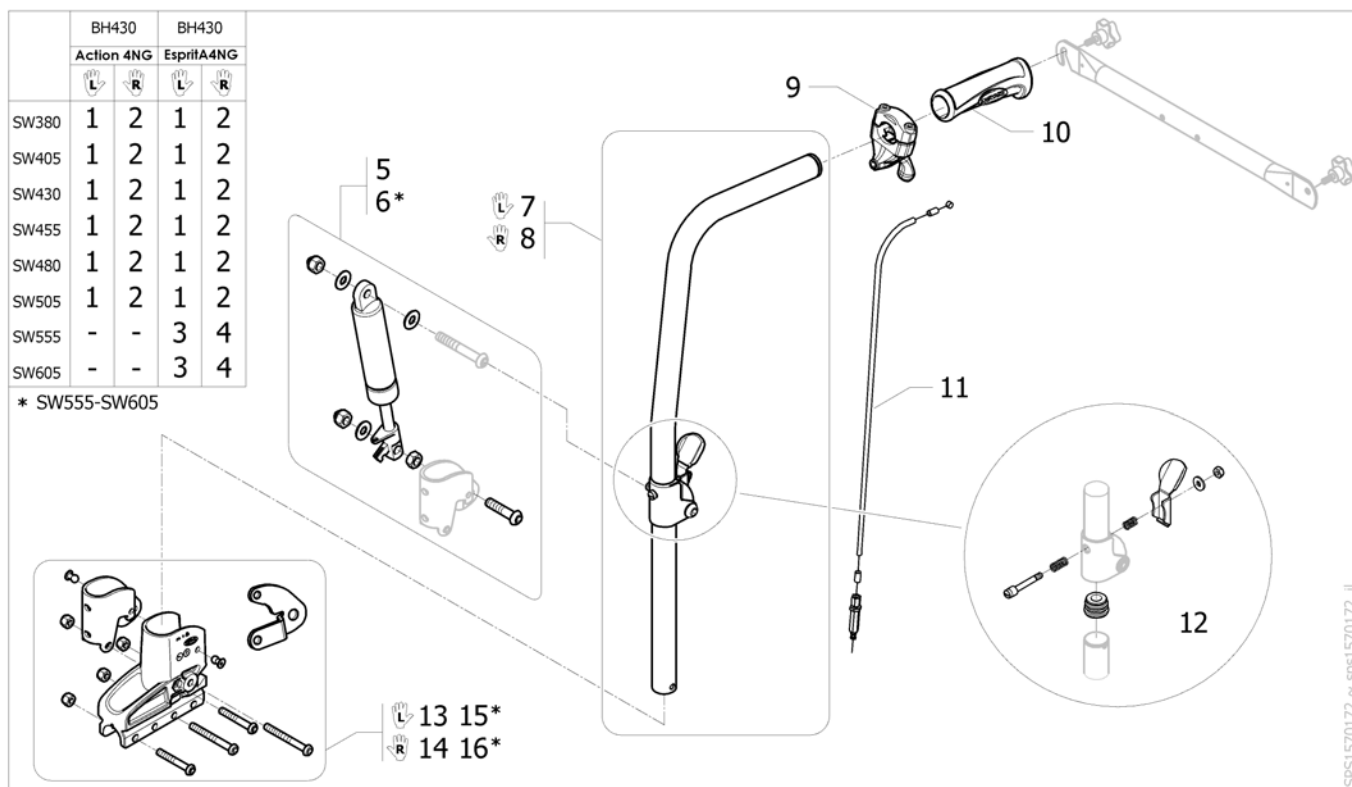


Pos.	Part number	Description	Valid from → until	Qty
1	1570175	Lh Reclining/folding backrest 0/30° gaz spring BH430 mm		1
2	1570176	Rh Reclining/folding backrest 0/30° gaz spring BH430 mm		1
3	1571487	Lh Reclining/folding backrest 0/30° gaz spring BH430 mm SW555-605 mm		1
4	1571488	Rh Reclining/folding backrest 0/30° gaz spring BH430 mm SW555-605 mm		1
5	1570300	Kit gas strut recliner/folding 0/30° BH430 mm		2
6	1572072	Kit gas strut recliner/folding 0/30° BH430 SW555-605 mm		1
7	1564695	Steel folding and Gaz spring Left		1
8	1564696	Steel folding and Gaz spring Right		1
9	1433747	Recliner backrest trigger		2
10	1425473	Handle Invacare for backrest Ø22mm		2
11	1440937	Recliner Gas Cable BH430 mm		2
12	1551064	Locking system for folding backrest	2013-03-04 →	2
13	1442513	Lh Recliner Gas Backrest Support Kit		1
14	1442507	Rh Recliner Gas Backrest Support Kit		1
15	1493418	Lh Recliner Gas Backrest Support Kit Hd		1
16	1493417	Rh Recliner Gas Backrest Support Kit Hd		1

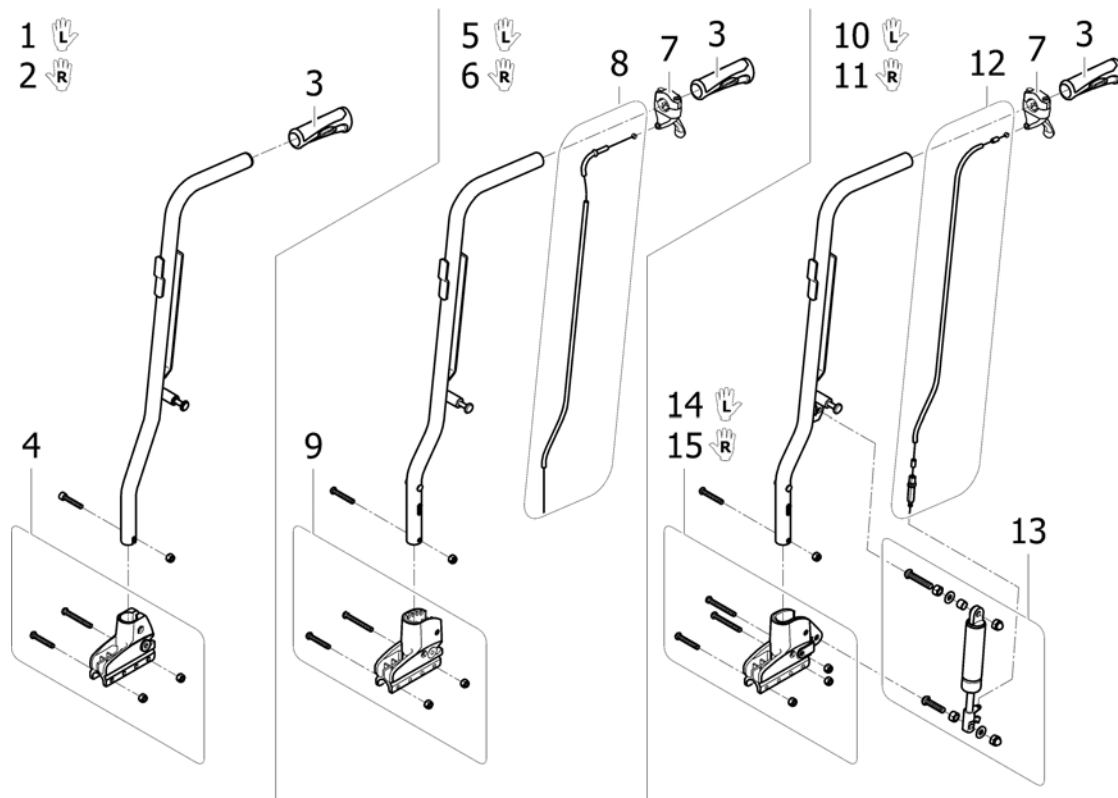Laguna backrests

ISPS1514263

Pos.	Part number	Description	Valid from → until	Qty
1	1515663	Kit Backrest post Laguna fixed LH + handle + support [CE_0421][CK_0313]		1
2	1515662	Kit Backrest post Laguna fixed RH + handle + support [CE_0421][CK_0313]		1
3	1425473	Handle Invacare for backrest Ø22mm Replace 1435421		2
4	1430848	Fixed Backrest Support Kit		2
5	1515661	Kit Backrest post Laguna recliner ratchet LH + handle + trigger + cable + support [CE_0441][CK_0343]		1
6	1515650	Kit Backrest post Laguna recliner ratchet RH + handle + trigger + cable + support [CE_0441][CK_0343]		1
7	1433747	Recliner backrest trigger		2
8	1430851	Recliner backrest cable Kit		2
9	1430850	Recliner Backrest Support Kit		2
10	1541873	Kit Backrest post Laguna recliner gas LH + handle + trigger + cable + gas spring + support [CE_0442][CK_0344]		1
11	1541872	Kit Backrest post Laguna recliner gas RH + handle + trigger + cable + gas spring + support [CE_0442][CK_0344]		1
12	1485871	Recliner Gas Cable BH510 mm		2
13	1532595	Gas Strut Kit Standard Backrest	2011-12-02 →	2
13	1528909	Gas Strut Kit Standard Backrest (pair)	2009-09-16 → 2011-12-01	1
14	1442513	Lh Recliner Gas Backrest Support Kit		1
15	1442507	Rh Recliner Gas Backrest Support Kit		1


Backrest upholsteries

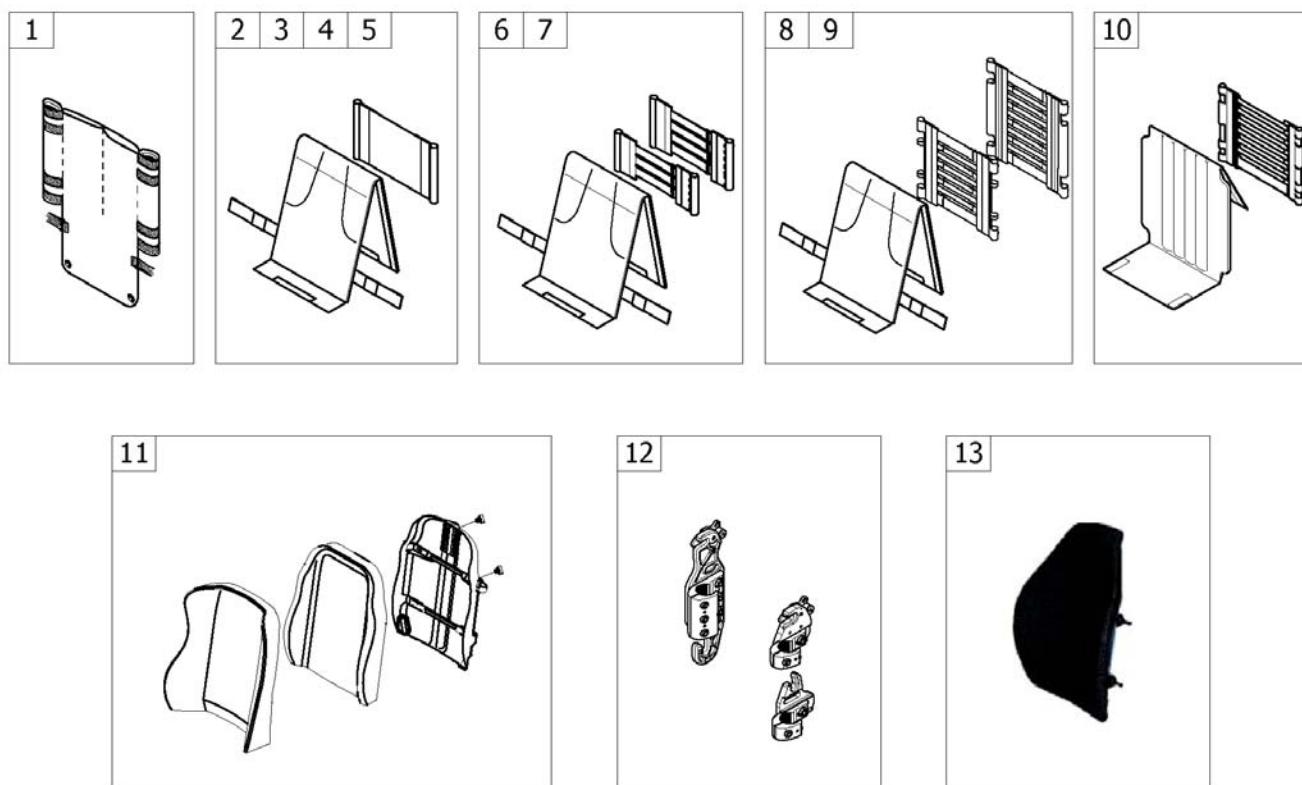

ISPS1574648

Pos.	Part number	Description	Valid from → until	Qty
1	-	Backrest upholstery Rea [CK_0360]		1
2	-	Lombar support padded back upholstery [CK_0365]		1
3	-	Lombar support padded back upholstery Bimaterial [CK_0364]		1
4	-	Backrest Upholstery For Height Adjustable Backrest [CK_0365+CK_0314] [CK_0365+CK_0316]		1
5	-	Backrest Upholstery Bimaterial For Height Adjustable Backrest [CK_0364+CK_0314] [CK_0364+CK_0316]		1
6	-	Contour adjustable padded back upholstery [CK_0369]		1
7	-	Contour adjustable Bimaterial padded back upholstery [CK_0368]		1
8	-	Contour adjustable nylon type back upholstery [CK_0370]		1
9	-	Contour adjustable Bimaterial nylon type back upholstery [CK_0375]		1
10	-	Contour adjustable backrest upholstery Rea [CK_0370+CK_0300] [CK_0370+CK_0310]		1
11	-	Comfort upholstery [CK_0371/CK_0372/CK_0374]		1
12	-	Mounting Hardware MatrX		1
13	-	Backrest upholstery MatrX		1

	Action 3NG	Action 4NG
	BH430	BH430
SW380	1	1
SW405	2	2
SW430	3	3
SW455	4	4
SW480	5	5
SW505	6	6
SW555	-	7
SW605	-	8

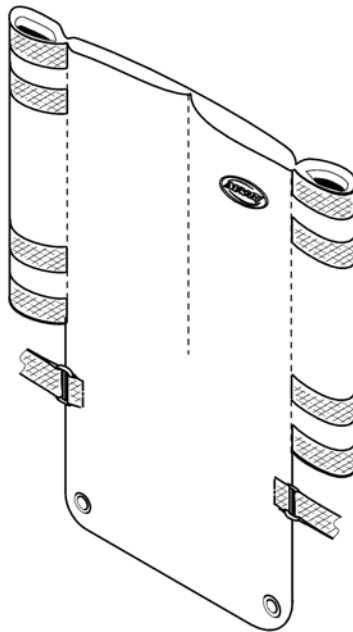

ISPS1496271

Pos.	Part number	Description	Valid from → until	Qty
1	1449809	Rea Backrest Upholstery SW380 x BH430 mm [CK_0360]		1
2	1449810	Rea Backrest Upholstery SW405 x BH430 mm [CK_0360]		1
3	1449811	Rea Backrest Upholstery SW430 x BH430 mm [CK_0360]		1
4	1449812	Rea Backrest Upholstery SW455 x BH430 mm [CK_0360]		1
5	1449813	Rea Backrest Upholstery SW480 x BH430 mm [CK_0360]		1
6	1449814	Rea Backrest Upholstery SW505 x BH430 mm [CK_0360]		1
7	1483908	Rea Backrest Upholstery SW555 x BH430 mm [CK_0360]		1
8	1483909	Rea Backrest Upholstery SW605 x BH430 mm [CK_0360]		1

Lombar support padded back upholstery

	Action 3NG EspritA4NG		Action 4NG	
	BH430	BH510	BH430	BH510
SW380	1	9	1	9
SW405	2	10	2	10
SW430	3	11	3	11
SW455	4	12	4	12
SW480	5	13	5	13
SW505	6	14	6	14
SW555	-	-	7	15
SW605	-	-	8	16

	Action 3NG EspritA4NG	Action 4NG
SW380	17	17
SW405	18	18
SW430	19	19
SW455	20	20
SW480	21	21
SW505	22	22
SW555	-	23
SW605	-	24


1531535387

Pos.	Part number	Description	Valid from → until	Qty
1	1532610	Lombar support padded back upholstery complete SW380 x BH430 mm [CK_0365]		1
2	1532611	Lombar support padded back upholstery complete SW405 x BH430 mm [CK_0365]		1
3	1532612	Lombar support padded back upholstery complete SW430 x BH430 mm [CK_0365]		1
4	1532613	Lombar support padded back upholstery complete SW455 x BH430 mm [CK_0365]		1
5	1532614	Lombar support padded back upholstery complete SW480 x BH430 mm [CK_0365]		1
6	1532615	Lombar support padded back upholstery complete SW505 x BH430 mm [CK_0365]		1
7	1532616	Lombar support padded back upholstery complete SW555 x BH430 mm [CK_0365]		1
8	1532617	Lombar support padded back upholstery complete SW605 x BH430 mm [CK_0365]		1
9	1532618	Lombar support padded back upholstery complete SW380 x BH510 mm [CK_0365]		1
10	1532619	Lombar support padded back upholstery complete SW405 x BH510 mm [CK_0365]		1
11	1532620	Lombar support padded back upholstery complete SW430 x BH510 mm [CK_0365]		1
12	1532621	Lombar support padded back upholstery complete SW455 x BH510 mm [CK_0365]		1
13	1532622	Lombar support padded back upholstery complete SW480 x BH510 mm [CK_0365]		1
14	1532623	Lombar support padded back upholstery complete SW505 x BH510 mm [CK_0365]		1
15	1532624	Lombar support padded back upholstery complete SW555 x BH510 mm [CK_0365]		1
16	1532625	Lombar support padded back upholstery complete SW605 x BH510 mm [CK_0365]		1
17	1530914	Shapped cover Action NG SW380 mm		1
18	1530915	Shapped cover Action NG SW405 mm		1
19	1531053	Shapped cover Action NG SW430 mm		1
20	1531054	Shapped cover Action NG SW455 mm		1

Lombar support padded back upholstery

Pos.	Part number	Description	Valid from → until	Qty
21	1531055	Shapped cover Action NG SW480 mm		1
22	1531056	Shapped cover Action NG SW505 mm		1
23	1531074	Shapped cover Action NG SW555 mm		1
24	1531075	Shapped cover Action NG SW605 mm		1
25	1428603	Screw Tfb 4,8x19 A2r Tx25 DIN7983 <i>Replace 0029-40005</i>		2

Lombar support padded back upholstery Bimaterial

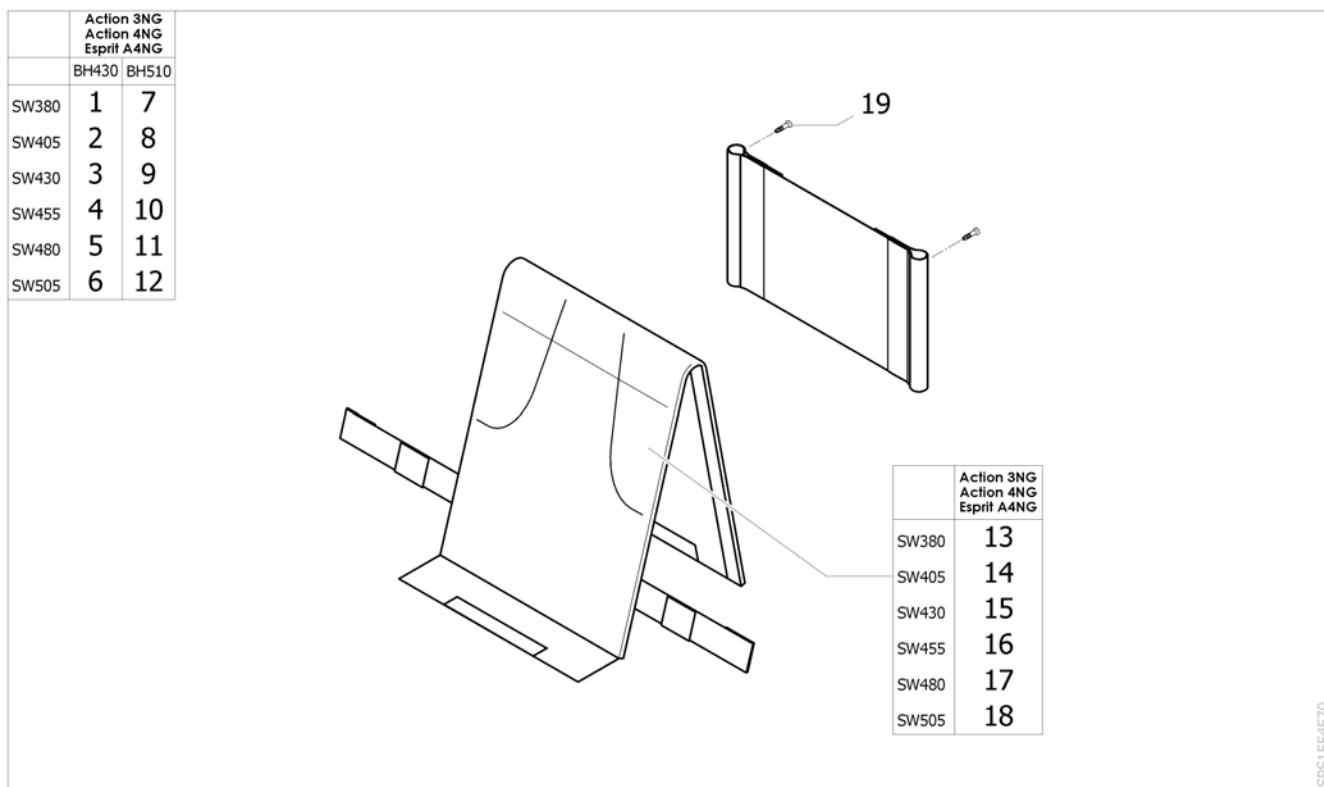


Pos.	Part number	Description	Valid from → until	Qty
1	1553458	Lombar support padded back upholstery Bimaterial complete SW380 x BH430 mm [CE_0474][CK_0364]		1
2	1553459	Lombar support padded back upholstery Bimaterial complete SW405 x BH430 mm [CE_0474][CK_0364]		1
3	1553460	Lombar support padded back upholstery Bimaterial complete SW430 x BH430 mm [CE_0474][CK_0364]		1
4	1553461	Lombar support padded back upholstery Bimaterial complete SW455 x BH430 mm [CE_0474][CK_0364]		1
5	1553462	Lombar support padded back upholstery Bimaterial complete SW480 x BH430 mm [CE_0474][CK_0364]		1
6	1553463	Lombar support padded back upholstery Bimaterial complete SW505 x BH430 mm [CE_0474][CK_0364]		1
7	1553466	Lombar support padded back upholstery Bimaterial complete SW380 x BH510 mm [CE_0474][CK_0364]		1
8	1553467	Lombar support padded back upholstery Bimaterial complete SW405 x BH510 mm [CE_0474][CK_0364]		1
9	1553468	Lombar support padded back upholstery Bimaterial complete SW430 x BH510 mm [CE_0474][CK_0364]		1
10	1553469	Lombar support padded back upholstery Bimaterial complete SW455 x BH510 mm [CE_0474][CK_0364]		1
11	1553470	Lombar support padded back upholstery Bimaterial complete SW480 x BH510 mm [CE_0474][CK_0364]		1
12	1553471	Lombar support padded back upholstery Bimaterial complete SW505 x BH510 mm [CE_0474][CK_0364]		1
13	1544272	Shapped cover Bimat�rial ActionNG SW380 mm		1
14	1544273	Shapped cover Bimat�rial ActionNG SW405 mm		1

Lombar support padded back upholstery Bimaterial


Pos.	Part number	Description	Valid from → until	Qty
15	1544274	Shapped cover Bimatériau ActionNG SW430 mm		1
16	1544275	Shapped cover Bimatériau ActionNG SW455 mm		1
17	1544276	Shapped cover Bimatériau ActionNG SW480 mm		1
18	1544277	Shapped cover Bimatériau ActionNG SW505 mm		1
19	1428603	Screw Tfb 4,8x19 A2r Tx25 DIN7983		2

Backrest Upholstery For Height Adjustable Backrest

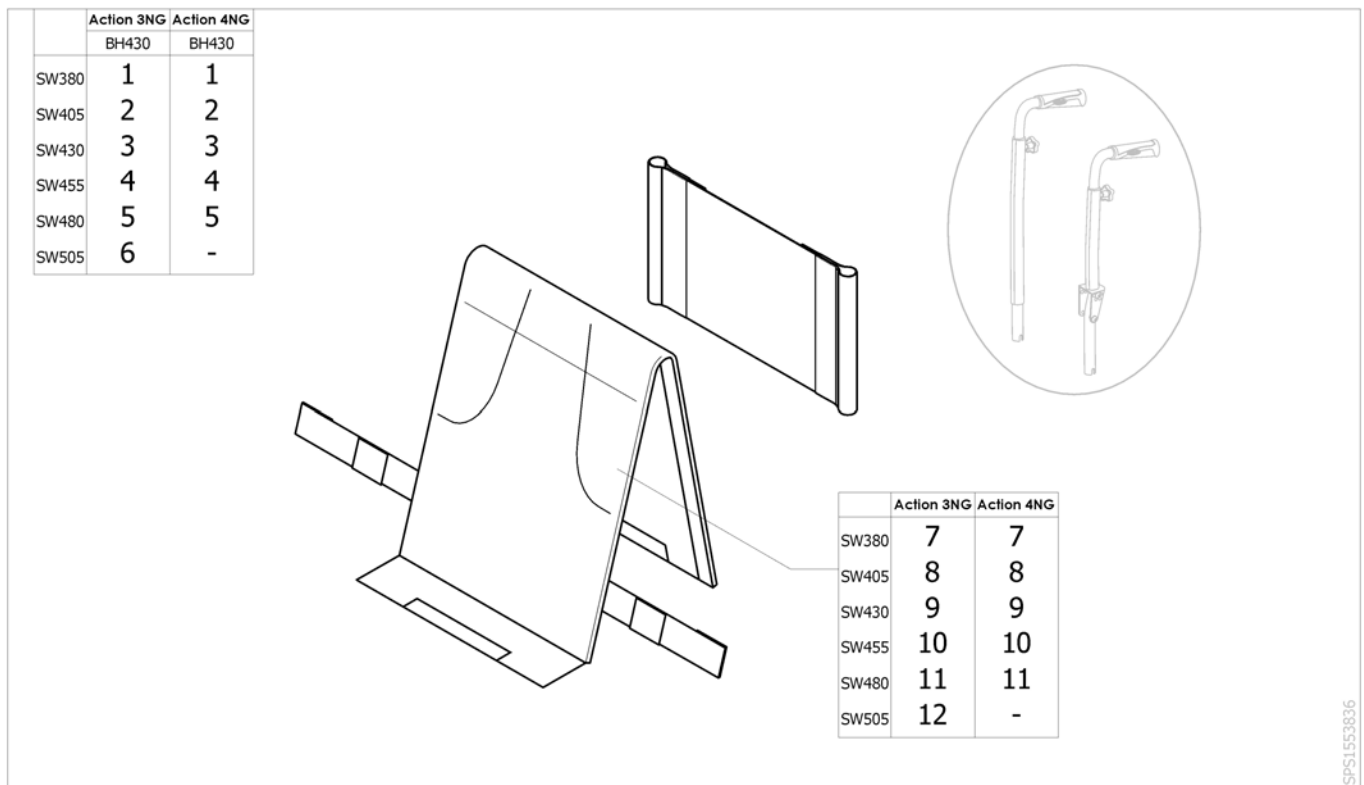

Pos.	Part number	Description	Valid from → until	Qty
1	1533872	Lombar support padded back upholstery complete for height adjustable handle SW380 x BH430 mm [CE_0475 + CE_0425/CE_0430]		1
2	1533922	Lombar support padded back upholstery complete for height adjustable handle SW405 x BH430 mm [CE_0475 + CE_0425/CE_0430]		1
3	1533923	Lombar support padded back upholstery complete for height adjustable handle SW430 x BH430 mm [CE_0475 + CE_0425/CE_0430]		1
4	1533924	Lombar support padded back upholstery complete for height adjustable handle SW455 x BH430 mm [CE_0475 + CE_0425/CE_0430]		1
5	1533925	Lombar support padded back upholstery complete for height adjustable handle SW480 x BH430 mm [CE_0475 + CE_0425/CE_0430]		1
6	1533926	Lombar support padded back upholstery complete for height adjustable handle SW505 x BH430 mm [CE_0475 + CE_0425/CE_0430]		1
7	1530914	Shapped cover Action NG SW380 mm		1
8	1530915	Shapped cover Action NG SW405 mm		1
9	1531053	Shapped cover Action NG SW430 mm		1
10	1531054	Shapped cover Action NG SW455 mm		1
11	1531055	Shapped cover Action NG SW480 mm		1
12	1531056	Shapped cover Action NG SW505 mm		1

Backrest Upholstery Bimaterial For Height Adjustable Backrest

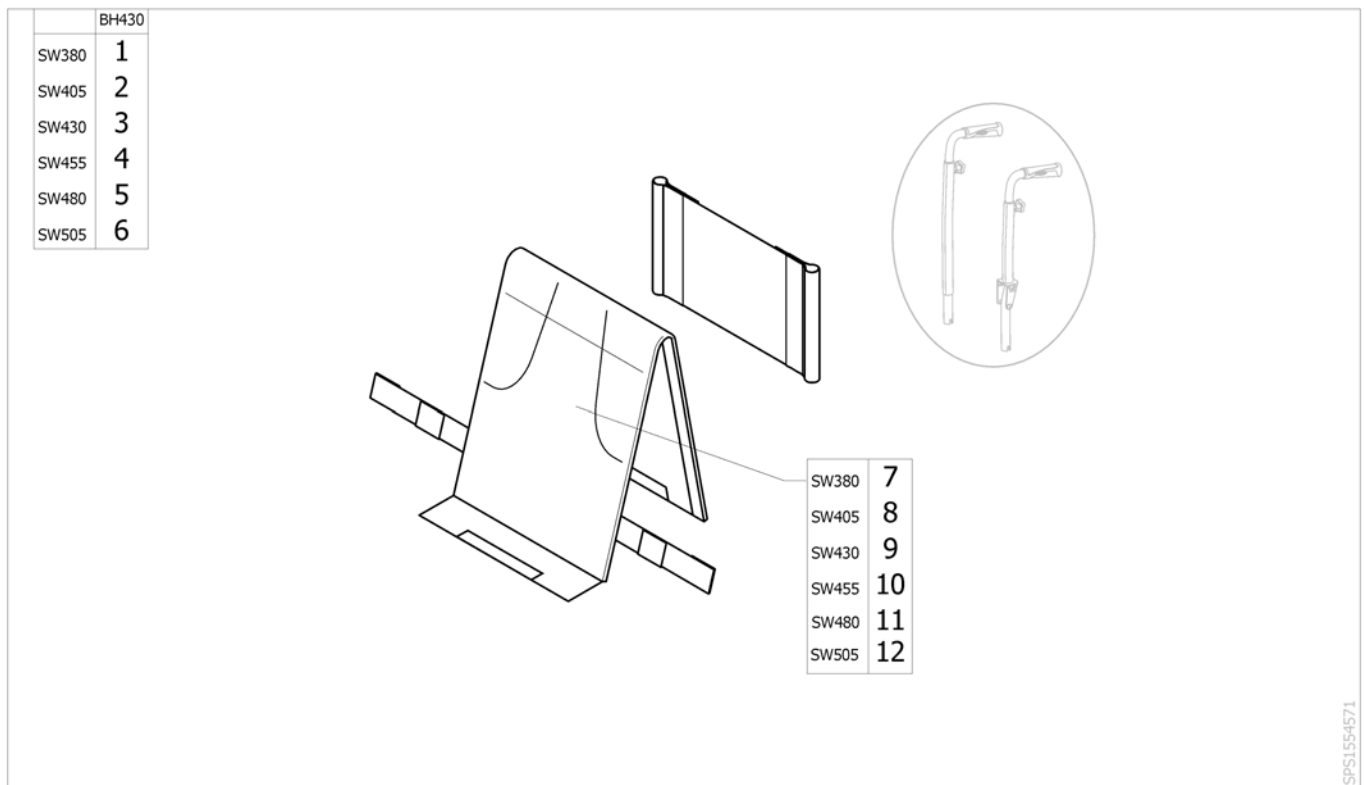

Pos.	Part number	Description	Valid from → until	Qty
1	1553684	Lombar support padded back upholstery Bimaterial complete for height adjustable handle SW380 x BH430 mm [CE_0474+CE_0425/CE_0430][CK_0364+CK_0314/CK_0316]		1
2	1553685	Lombar support padded back upholstery Bimaterial complete for height adjustable handle SW405 x BH430 mm [CE_0474+CE_0425/CE_0430][CK_0364+CK_0314/CK_0316]		1
3	1553724	Lombar support padded back upholstery Bimaterial complete for height adjustable handle SW430 x BH430 mm [CE_0474+CE_0425/CE_0430][CK_0364+CK_0314/CK_0316]		1
4	1553725	Lombar support padded back upholstery Bimaterial complete for height adjustable handle SW455 x BH430 mm [CE_0474+CE_0425/CE_0430][CK_0364+CK_0314/CK_0316]		1
5	1553726	Lombar support padded back upholstery Bimaterial complete for height adjustable handle SW480 x BH430 mm [CE_0474+CE_0425/CE_0430][CK_0364+CK_0314/CK_0316]		1
6	1553727	Lombar support padded back upholstery Bimaterial complete for height adjustable handle SW505 x BH430 mm [CE_0474+CE_0425/CE_0430][CK_0364+CK_0314/CK_0316]		1
7	1544272	Shapped cover Bimat�rial ActionNG SW380 mm		1
8	1544273	Shapped cover Bimat�rial ActionNG SW405 mm		1
9	1544274	Shapped cover Bimat�rial ActionNG SW430 mm		1
10	1544275	Shapped cover Bimat�rial ActionNG SW455 mm		1
11	1544276	Shapped cover Bimat�rial ActionNG SW480 mm		1
12	1544277	Shapped cover Bimat�rial ActionNG SW505 mm		1

Contour adjustable padded back upholstery

	Action 3NG			Action 4NG		
	BH350	BH430	BH510	BH350	BH430	BH510
SW380	1	1	9	1	1	9
SW405	2	2	10	2	2	10
SW430	3	3	11	3	3	11
SW455	4	4	12	4	4	12
SW480	5	5	13	5	5	13
SW505	6	6	14	-	6	14
SW555	-	-	-	-	7	15
SW605	-	-	-	-	8	16

	Action 3NG	Action 4NG
SW380	17	17
SW405	18	18
SW430	19	19
SW455	20	20
SW480	21	21
SW505	22	22
SW555	-	23
SW605	-	24


1532667

Pos.	Part number	Description	Valid from → until	Qty
1	1532667	Contour adjustable padded back upholstery complete 3 Straps SW380 x BH430 mm [CK_0369]		1
2	1532686	Contour adjustable padded back upholstery complete 3 Straps SW405 x BH430 mm [CK_0369]		1
3	1532687	Contour adjustable padded back upholstery complete 3 Straps SW430 x BH430 mm [CK_0369]		1
4	1532779	Contour adjustable padded back upholstery complete 3 Straps SW455 x BH430 mm [CK_0369]		1
5	1532780	Contour adjustable padded back upholstery complete 3 Straps SW480 x BH430 mm [CK_0369]		1
6	1532781	Contour adjustable padded back upholstery complete 3 Straps SW505 x BH430 mm [CK_0369]		1
7	1532782	Contour adjustable padded back upholstery complete 3 Straps SW555 x BH430 mm [CK_0369]		1
8	1532783	Contour adjustable padded back upholstery complete 3 Straps SW605 x BH430 mm [CK_0369]		1
9	1532784	Contour adjustable padded back upholstery complete 4 Straps SW380 x BH510 mm [CK_0369]		1
10	1532785	Contour adjustable padded back upholstery complete 4 Straps SW405 x BH510 mm [CK_0369]		1
11	1532786	Contour adjustable padded back upholstery complete 4 Straps SW430 x BH510 mm [CK_0369]		1
12	1532787	Contour adjustable padded back upholstery complete 4 Straps SW455 x BH510 mm [CK_0369]		1

Contour adjustable padded back upholstery

Pos.	Part number	Description	Valid from → until	Qty
13	1532788	Contour adjustable padded back upholstery complete 4 Straps SW480 x BH510 mm <i>[CK_0369]</i>		1
14	1532789	Contour adjustable padded back upholstery complete 4 Straps SW505 x BH510 mm <i>[CK_0369]</i>		1
15	1532790	Contour adjustable padded back upholstery complete 4 Straps SW555 x BH510 mm <i>[CK_0369]</i>		1
16	1532791	Contour adjustable padded back upholstery complete 4 Straps SW605 x BH510 mm <i>[CK_0369]</i>		1
17	1530914	Shapped cover Action NG SW380 mm		1
18	1530915	Shapped cover Action NG SW405 mm		1
19	1531053	Shapped cover Action NG SW430 mm		1
20	1531054	Shapped cover Action NG SW455 mm		1
21	1531055	Shapped cover Action NG SW480 mm		1
22	1531056	Shapped cover Action NG SW505 mm		1
23	1531074	Shapped cover Action NG SW555 mm		1
24	1531075	Shapped cover Action NG SW605 mm		1
25	1428603	Screw Tfb 4,8x19 A2r Tx25 DIN7983 <i>Replace 0029-40005</i>		2

Contour adjustable Bimaterial padded back upholstery

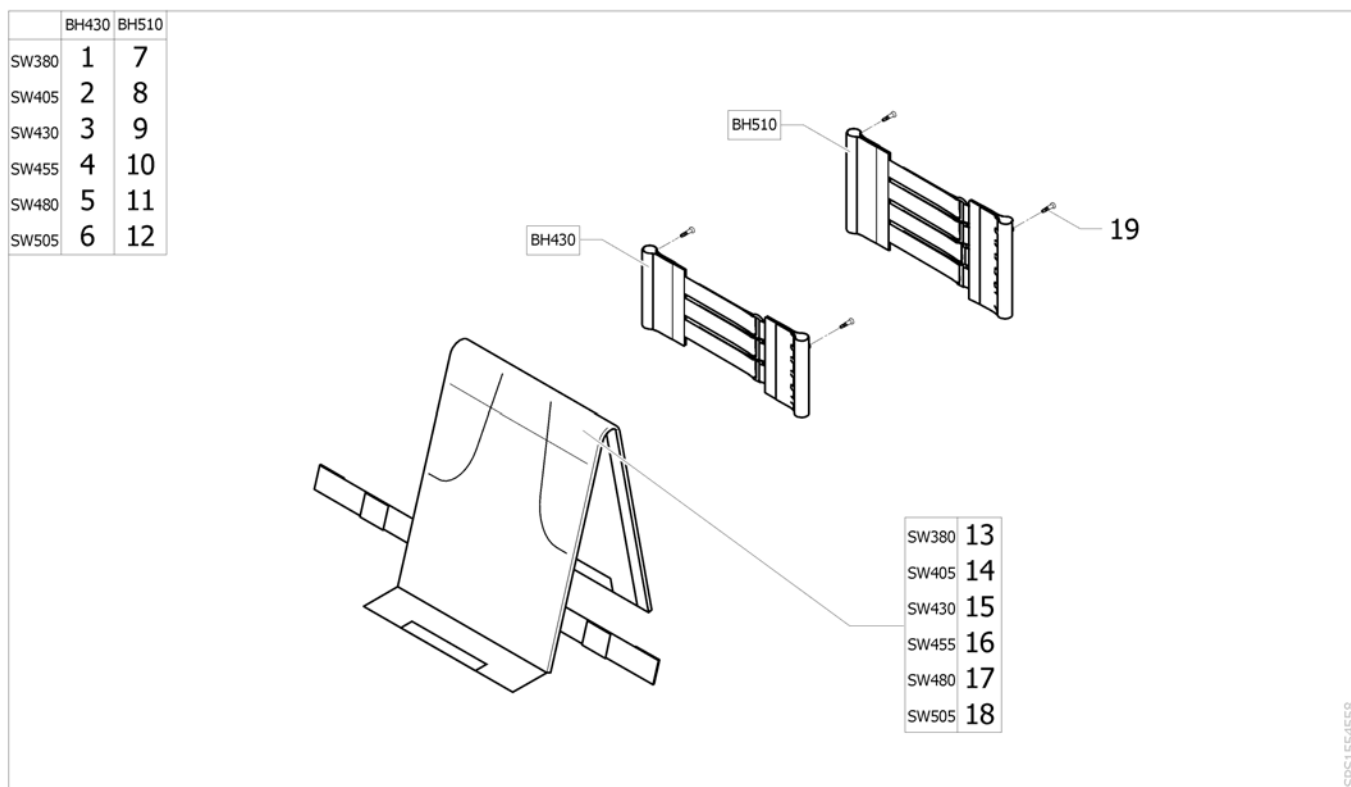

Pos.	Part number	Description	Valid from → until	Qty
1	1553480	Contour adjustable padded back upholstery Bimaterial complete 3 Straps SW380 x BH430 mm [CE_0478][CK_0368]		1
2	1553481	Contour adjustable padded back upholstery Bimaterial complete 3 Straps SW405 x BH430 mm [CE_0478][CK_0368]		1
3	1553482	Contour adjustable padded back upholstery Bimaterial complete 3 Straps SW430 x BH430 mm [CE_0478][CK_0368]		1
4	1553483	Contour adjustable padded back upholstery Bimaterial complete 3 Straps SW455 x BH430 mm [CE_0478][CK_0368]		1
5	1553505	Contour adjustable padded back upholstery Bimaterial complete 3 Straps SW480 x BH430 mm [CE_0478][CK_0368]		1
6	1553545	Contour adjustable padded back upholstery Bimaterial complete 3 Straps SW505 x BH430 mm [CE_0478][CK_0368]		1
7	1553549	Contour adjustable padded back upholstery Bimaterial complete 4 Straps SW380 x BH510 mm [CE_0478][CK_0368]		1
8	1553550	Contour adjustable padded back upholstery Bimaterial complete 4 Straps SW405 x BH510 mm [CE_0478][CK_0368]		1
9	1553551	Contour adjustable padded back upholstery Bimaterial complete 4 Straps SW430 x BH510 mm [CE_0478][CK_0368]		1
10	1553552	Contour adjustable padded back upholstery Bimaterial complete 4 Straps SW455 x BH510 mm [CE_0478][CK_0368]		1
11	1553562	Contour adjustable padded back upholstery Bimaterial complete 4 Straps SW480 x BH510 mm [CE_0478][CK_0368]		1
12	1553563	Contour adjustable padded back upholstery Bimaterial complete 4 Straps SW505 x BH510 mm [CE_0478][CK_0368]		1
13	1544272	Shapped cover Bimaterial ActionNG SW380 mm		1
14	1544273	Shapped cover Bimaterial ActionNG SW405 mm		1

Contour adjustable Bimaterial padded back upholstery

Pos.	Part number	Description	Valid from → until	Qty
15	1544274	Shapped cover Bimatériel ActionNG SW430 mm		1
16	1544275	Shapped cover Bimatériel ActionNG SW455 mm		1
17	1544276	Shapped cover Bimatériel ActionNG SW480 mm		1
18	1544277	Shapped cover Bimatériel ActionNG SW505 mm		1
19	1428603	Screw Tfb 4,8x19 A2r Tx25 DIN7983		2

Contour adjustable nylon type back upholstery

	Action 3NG		Action 4NG	
	BH430	BH510	BH430	BH510
SW380	1	9	1	9
SW405	2	10	2	10
SW430	3	11	3	11
SW455	4	12	4	12
SW480	5	13	5	13
SW505	6	14	6	14
SW555	-	-	7	-
SW605	-	-	8	-

	Action 3NG	Action 4NG
SW380	15	15
SW405	16	16
SW430	17	17
SW455	18	18
SW480	19	19
SW505	20	20
SW555	-	21
SW605	-	22


ISPS1535389

Pos.	Part number	Description	Valid from → until	Qty
1	1533860	Contour adjustable nylon type back upholstery complete 8 Straps 4 Loops SW380 x BH430 mm [CK_0370]		1
2	1533861	Contour adjustable nylon type back upholstery complete 8 Straps 4 Loops SW405 x BH430 mm [CK_0370]		1
3	1533862	Contour adjustable nylon type back upholstery complete 8 Straps 4 Loops SW430 x BH430 mm [CK_0370]		1
4	1533863	Contour adjustable nylon type back upholstery complete 8 Straps 4 Loops SW455 x BH430 mm [CK_0370]		1
5	1533864	Contour adjustable nylon type back upholstery complete 8 Straps 4 Loops SW480 x BH430 mm [CK_0370]		1
6	1533865	Contour adjustable nylon type back upholstery complete 8 Straps 4 Loops SW505 x BH430 mm [CK_0370]		1
7	1530782	Contour adjustable nylon type back upholstery complete 8 Straps 4 Loops SW555 x BH430 mm [CK_0370]		1
8	1530783	Contour adjustable nylon type back upholstery complete 8 Straps 4 Loops SW605 x BH430 mm [CK_0370]		1
9	1533866	Contour adjustable nylon type back upholstery complete 10 Straps 3 Loops SW380 x BH510 mm [CK_0370]		1
10	1533867	Contour adjustable nylon type back upholstery complete 10 Straps 3 Loops SW405 x BH510 mm [CK_0370]		1
11	1533868	Contour adjustable nylon type back upholstery complete 10 Straps 3 Loops SW430 x BH510 mm [CK_0370]		1
12	1533869	Contour adjustable nylon type back upholstery complete 10 Straps 3 Loops SW455 x BH510 mm [CK_0370]		1

Contour adjustable nylon type back upholstery

Pos.	Part number	Description	Valid from → until	Qty
13	1533870	Contour adjustable nylon type back upholstery complete 10 Straps 3 Loops SW480 x BH510 mm <i>[CK_0370]</i>		1
14	1533871	Contour adjustable nylon type back upholstery complete 10 Straps 3 Loops SW505 x BH510 mm <i>[CK_0370]</i>		1
15	1530914	Shapped cover Action NG SW380 mm		1
16	1530915	Shapped cover Action NG SW405 mm		1
17	1531053	Shapped cover Action NG SW430 mm		1
18	1531054	Shapped cover Action NG SW455 mm		1
19	1531055	Shapped cover Action NG SW480 mm		1
20	1531056	Shapped cover Action NG SW505 mm		1
21	1531074	Shapped cover Action NG SW555 mm		1
22	1531075	Shapped cover Action NG SW605 mm		1
23	1428603	Screw Tfb 4,8x19 A2r Tx25 DIN7983 <i>Replace 0029-40005</i>		2

Contour adjustable Bimaterial nylon type back upholstery

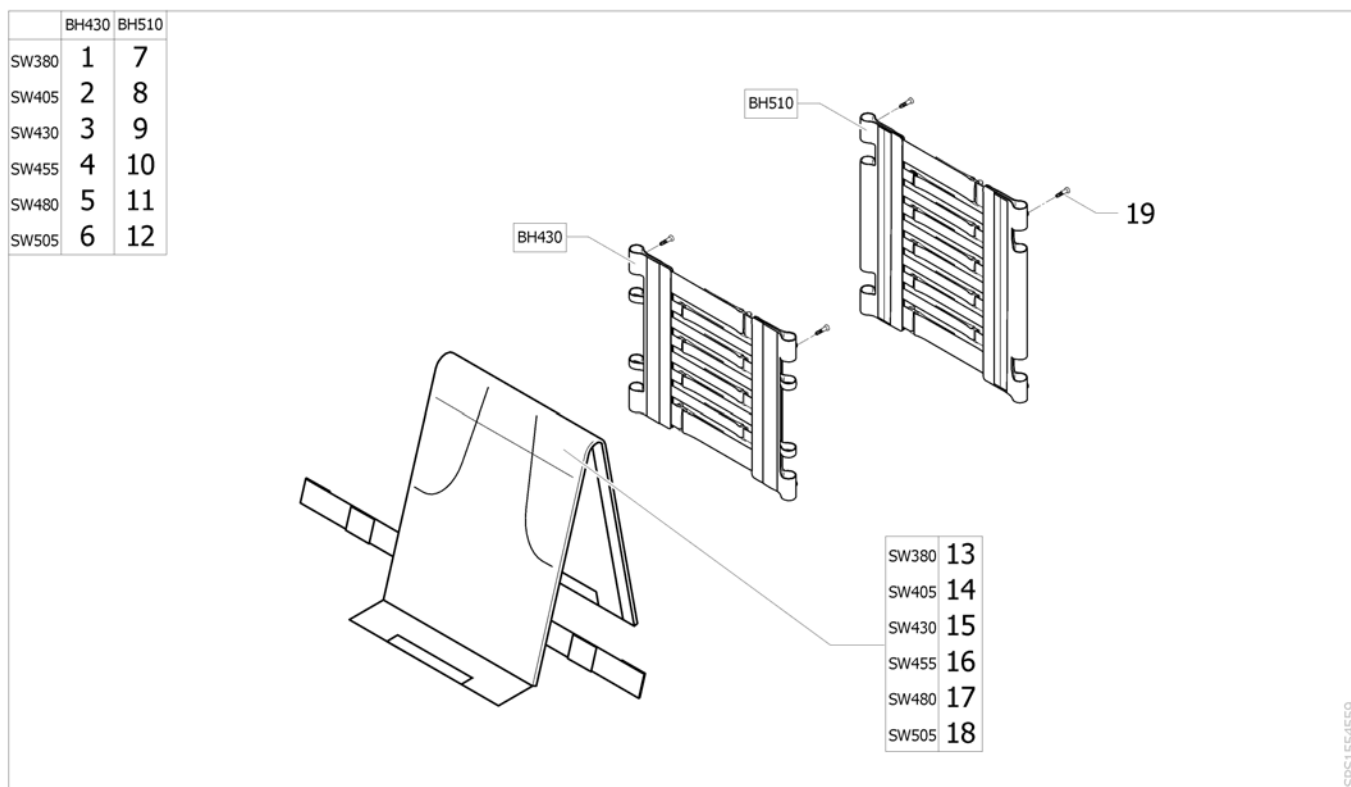


Pos.	Part number	Description	Valid from → until	Qty
1	1553566	Contour adjustable nylon type back upholstery Bimaterial complete 8 Straps 4 Loops SW380 x BH430 mm [CE_0481][CK_0375]		1
2	1553567	Contour adjustable nylon type back upholstery Bimaterial complete 8 Straps 4 Loops SW405 x BH430 mm [CE_0481][CK_0375]		1
3	1553568	Contour adjustable nylon type back upholstery Bimaterial complete 8 Straps 4 Loops SW430 x BH430 mm [CE_0481][CK_0375]		1
4	1553569	Contour adjustable nylon type back upholstery Bimaterial complete 8 Straps 4 Loops SW455 x BH430 mm [CE_0481][CK_0375]		1
5	1553570	Contour adjustable nylon type back upholstery Bimaterial complete 8 Straps 4 Loops SW480 x BH430 mm [CE_0481][CK_0375]		1
6	1553571	Contour adjustable nylon type back upholstery Bimaterial complete 8 Straps 4 Loops SW505 x BH430 mm [CE_0481][CK_0375]		1
7	1553588	Contour adjustable nylon type back upholstery Bimaterial complete 10 Straps 3 Loops SW380 x BH510 mm [CE_0481][CK_0375]		1
8	1553589	Contour adjustable nylon type back upholstery Bimaterial complete 10 Straps 3 Loops SW405 x BH510 mm [CE_0481][CK_0375]		1
9	1553590	Contour adjustable nylon type back upholstery Bimaterial complete 10 Straps 3 Loops SW430 x BH510 mm [CE_0481][CK_0375]		1
10	1553679	Contour adjustable nylon type back upholstery Bimaterial complete 10 Straps 3 Loops SW455 x BH510 mm [CE_0481][CK_0375]		1
11	1553680	Contour adjustable nylon type back upholstery Bimaterial complete 10 Straps 3 Loops SW480 x BH510 mm [CE_0481][CK_0375]		1
12	1553681	Contour adjustable nylon type back upholstery Bimaterial complete 10 Straps 3 Loops SW505 x BH510 mm [CE_0481][CK_0375]		1
13	1544272	Shapped cover Bimaterial ActionNG SW380 mm		1
14	1544273	Shapped cover Bimaterial ActionNG SW405 mm		1


Contour adjustable Bimaterial nylon type back upholstery

Pos.	Part number	Description	Valid from → until	Qty
15	1544274	Shapped cover Bimatériau ActionNG SW430 mm		1
16	1544275	Shapped cover Bimatériau ActionNG SW455 mm		1
17	1544276	Shapped cover Bimatériau ActionNG SW480 mm		1
18	1544277	Shapped cover Bimatériau ActionNG SW505 mm		1
19	1428603	Screw Tfb 4,8x19 A2r Tx25 DIN7983		2

Contour adjustable backrest upholstery Rea

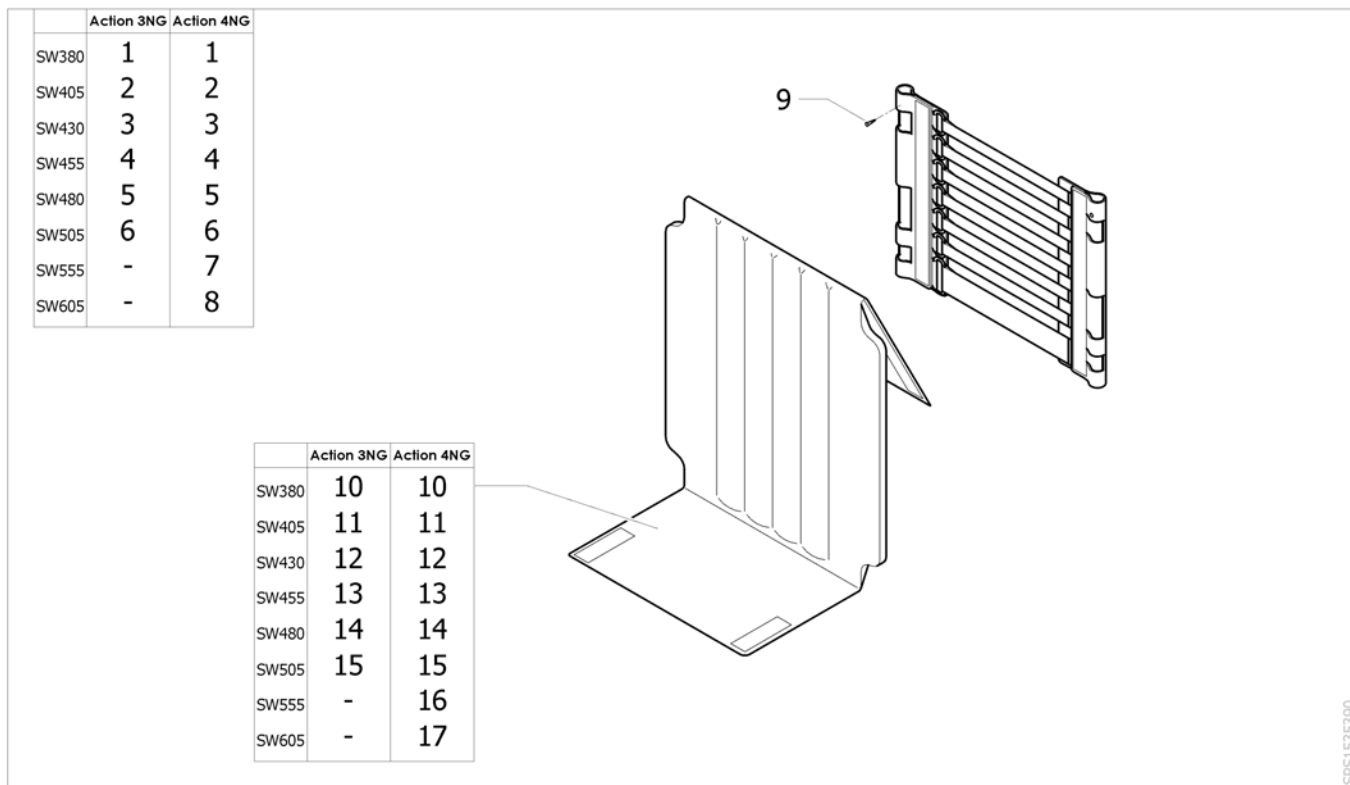

Pos.	Part number	Description	Valid from → until	Qty
1	1449965	Adjustable Back Upholstery 7 Straps SW380 x BH430 mm (Fixed) <i>[CK_0370+CK_0300/CK_0310]</i>		1
2	1449966	Adjustable Back Upholstery 7 Straps SW405 x BH430 mm (Fixed) <i>[CK_0370+CK_0300/CK_0310]</i>		1
3	1449967	Adjustable Back Upholstery 7 Straps SW430 x BH430 mm (Fixed) <i>[CK_0370+CK_0300/CK_0310]</i>		1
4	1449968	Adjustable Back Upholstery 7 Straps SW455 x BH430 mm (Fixed) <i>[CK_0370+CK_0300/CK_0310]</i>		1
5	1449969	Adjustable Back Upholstery 7 Straps SW480 x BH430 mm (Fixed) <i>[CK_0370+CK_0300/CK_0310]</i>		1
6	1449970	Adjustable Back Upholstery 7 Straps SW505 x BH430 mm (Fixed) <i>[CK_0370+CK_0300/CK_0310]</i>		1
7	1478720	Adjustable Back Upholstery 7 Straps SW555 x BH430 mm (Fixed) <i>[CK_0370+CK_0300/CK_0310]</i>		1
8	1478721	Adjustable Back Upholstery 7 Straps SW605 x BH430 mm (Fixed) <i>[CK_0370+CK_0300/CK_0310]</i>		1
9	1428603	Screw Tfb 4,8x19 A2r Tx25 DIN7983 <i>Replace 0029-40005</i>		2
10	1540578-TR18	Cover for Backrest adjustable SW380 Jemina <i>Replace 5106739-TR18</i>		1
11	1540579-TR18	Cover for Backrest adjustable SW405 Jemina <i>Replace 5106740-TR18</i>		1
12	1540580-TR18	Cover for Backrest adjustable SW430 Jemina <i>Replace 5106741-TR18</i>		1
13	1540581-TR18	Cover for Backrest adjustable SW455 Jemina <i>Replace 5106742-TR18</i>		1
14	1540582-TR18	Cover for Backrest adjustable SW480 Jemina <i>Replace 1435523</i>		1
15	1540583-TR18	Cover for Backrest adjustable SW505 Jemina <i>Replace 5106743-TR18</i>		1
16	1540585-TR18	A4 Adjustable Back Cover SW555 mm (Fixed) <i>Replace 1478716</i>		1
17	1540586-TR18	A4 Adjustable Back Cover SW605 mm (Fixed) <i>Replace 1478717</i>		1


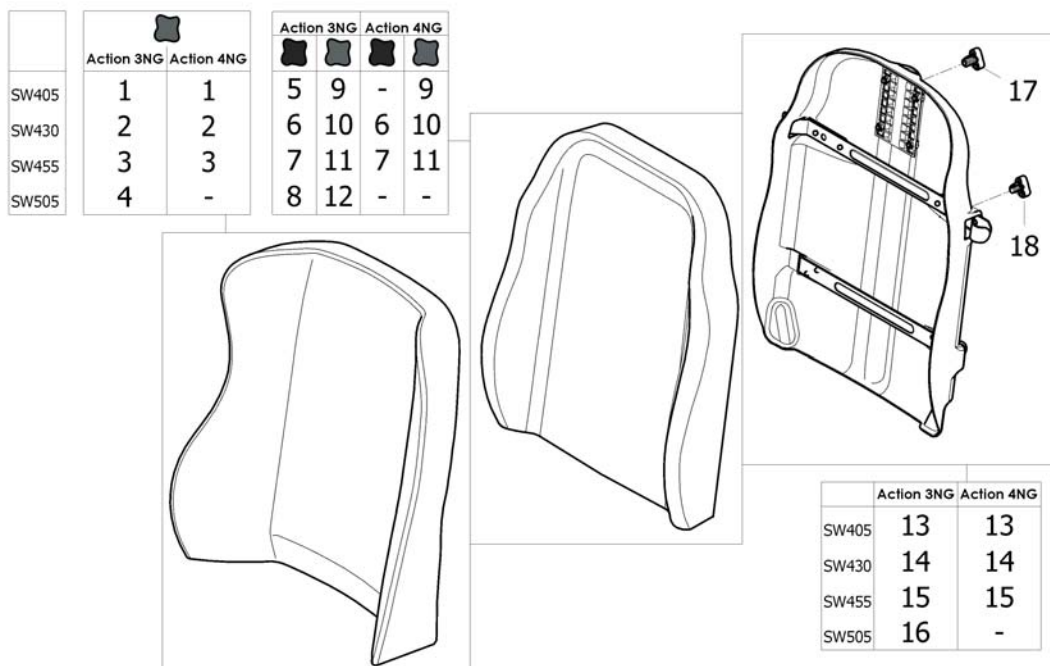
Comfort upholstery

ISPS1541874

Pos.	Part number	Description	Valid from → until	Qty
1	1440951-TR26	Backrest cushion comfort Passad2 (XXS), W405 x H550 mm Dartex Black (TR26) [CE_0491]		1
2	1429271-TR26	Backrest cushion comfort Passad2 (XS), W440 x H550 mm, Dartex black (TR26) [CE_0491]		1
3	1424647-TR26	Backrest cushion comfort Passad2 (S), W470 x H550 mm, Dartex black (TR26) [CE_0491]		1
4	1424649-TR26	Backrest cushion comfort Passad2 (M), W520 x H550 mm, Dartex black (TR26) [CE_0491]		1
5	1439937-TR26	Backrest cushion comfort Laguna (XXS), W342 x H550 mm, Dartex black (TR26) [CE_0495] - Replace 1431516		1
6	1427118-TR26	Backrest cushion comfort Laguna (XS), W372 x H550 mm, Dartex black (TR26) [CE_0495] - Replace 1420088		1
7	1427119-TR26	Backrest cushion comfort Laguna (S), W402 x H550 mm, Dartex black (TR26) [CE_0495] - Replace 1420089		1
8	1427120-TR26	Backrest cushion comfort Laguna (M), W452 x H550 mm, Dartex black (TR26) [CE_0495] - Replace 1420090		1
9	1439937-TR26	Backrest cushion comfort Laguna (XXS), W342 x H550 mm, Dartex black (TR26) [CE_0490]		1
10	1427118-TR26	Backrest cushion comfort Laguna (XS), W372 x H550 mm, Dartex black (TR26) [CE_0490]		1
11	1427119-TR26	Backrest cushion comfort Laguna (S), W402 x H550 mm, Dartex black (TR26) [CE_0490]		1
12	1427120-TR26	Backrest cushion comfort Laguna (M), W452 x H550 mm, Dartex black (TR26) [CE_0490]		1
13	1437532	Laguna Backrest Shell SW405 mm		1
14	1437534	Laguna Backrest Shell SW430 mm		1

Comfort upholstery

Pos.	Part number	Description	Valid from → until	Qty
15	1437536	Laguna Backrest Shell SW455 mm		1
16	1437538	Laguna Backrest Shell SW505 mm		1
17	1497411	HANDWHEEL M12 LONG NECK/HEADREST <i>Replace 1444116</i>		1
18	1440943	Locking star knob		1

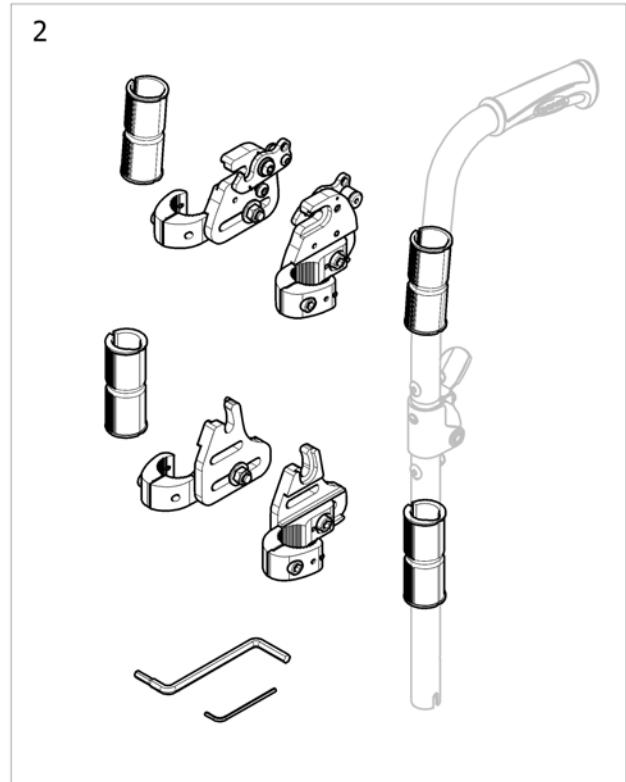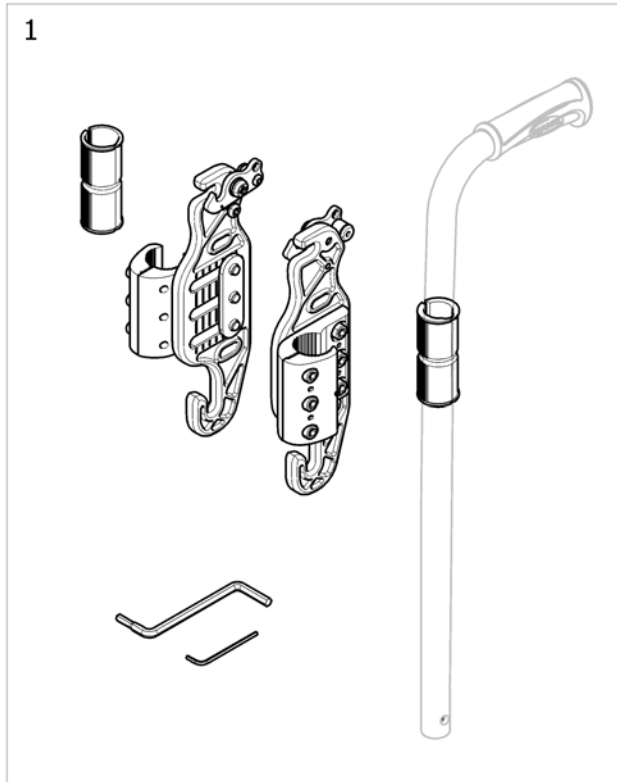

ISPS1574743

Pos.	Part number	Description	Valid from → until	Qty
1	1545275	PB Mounting Hardware for all MC Backs (pair)		2
2	1545278	Split mounting hardware suitable for all MC backs (pair)		2

Backrest upholstery MatrX

	Action 3NG EspritA4NG		Action 4NG	
	BH405	BH455	BH405	BH455
SW380	1	7	1	7
SW405	2	8	2	8
SW430	3	9	3	9
SW455	4	10	4	10
SW480	5	11	5	11
SW505	6	12	-	-





	Action 3NG EspritA4NG		Action 4NG	
	BH405	BH455	BH405	BH455
SW380	13	19	13	19
SW405	14	20	14	20
SW430	15	21	15	21
SW455	16	22	16	22
SW480	17	23	17	23
SW505	18	24	-	-

ISPS1574742


Pos.	Part number	Description	Valid from → until	Qty
1	1532093	Backrest MatrX Elite W380 x H410 mm - without hardware		1
2	1515720	Backrest MatrX Elite W410 x H410 mm - without hardware		1
3	1515722	Backrest MatrX Elite W430 x H410 mm - without hardware		1
4	1515724	Backrest MatrX Elite W360 x H410 mm - without hardware		1
5	1515726	Backrest MatrX Elite W480 x H410 mm - without hardware		1
6	1532096	Backrest MatrX Elite W510 x H410 mm - without hardware		1
7	1545100	Backrest MatrX Elite W380 x H460 mm - without hardware		1
8	1545102	Backrest MatrX Elite W410 x H460 mm - without hardware		1
9	1545105	Backrest MatrX Elite W430 x H460 mm - without hardware		1
10	1545108	Backrest MatrX Elite W460 x H460 mm - without hardware		1
11	1545111	Backrest MatrX Elite W480 x H460 mm - without hardware		1
12	1545114	Backrest MatrX Elite W510 x H460 mm - without hardware		1
13	1543509	Backrest MatrX Elite Deep W380 x H410 mm - without hardware		1
14	1543513	Backrest MatrX Elite Deep W410 x H410 mm - without hardware		1
15	1543517	Backrest MatrX Elite Deep W430 x H410 mm - without hardware		1
16	1543521	Backrest MatrX Elite Deep W460 x H410 mm - without hardware		1
17	1543525	Backrest MatrX Elite Deep W480 x H410 mm - without hardware		1
18	1543989	Backrest MatrX Elite Deep W510 x H410 mm - without hardware		1
19	1543510	Backrest MatrX Elite Deep W380 x H460 mm - without hardware		1
20	1543514	Backrest MatrX Elite Deep W410 x H460 mm - without hardware		1
21	1543518	Backrest MatrX Elite Deep W430 x H460 mm - without hardware		1
22	1543522	Backrest MatrX Elite Deep W460 x H460 mm - without hardware		1
23	1543526	Backrest MatrX Elite Deep W480 x H460 mm - without hardware		1
24	1543990	Backrest MatrX Elite Deep W510 x H460 mm - without hardware		1

Legrests

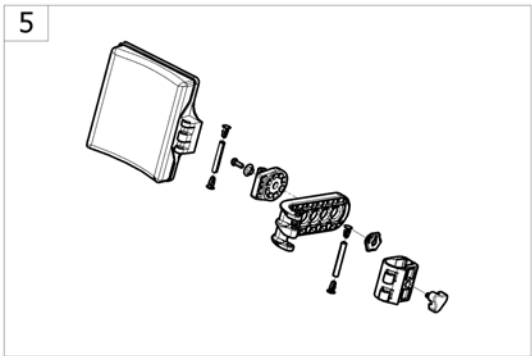
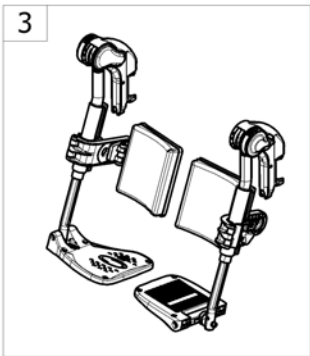


<SPACTION4NG30

Pos.	Part number	Description	Valid from → until	Qty
1	-	Hanger fixed 80°		1
2	-	Hanger fixed 80° =>2015.01.25		1
3	-	Assembly Legrest with footrest		1
4	-	Elevating legrest		1
5	-	Calf Pad		1
6	-	Amputee Legrest		1

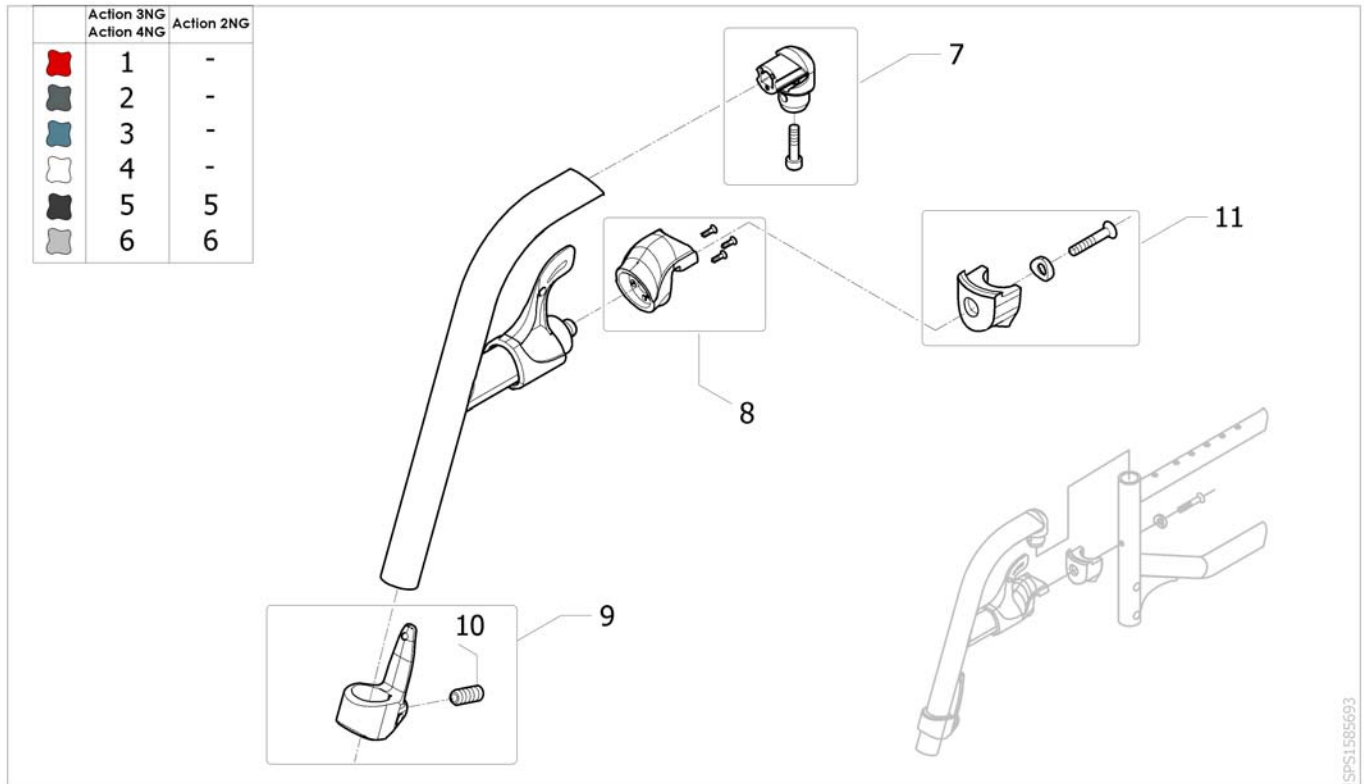
Hanger fixed 80°

Pos.	Part number	Description	Valid from → until	Qty
1	SP1584369-0607	Hanger 80°, complete, Dark Red (2015)		2
2	SP1584369-0615	Footrest hanger 80° Salte Grey(2015 Version)		2
3	SP1584369-0617	Footrest hanger 80° Océan Blue(2015 Version)		2
4	SP1584369-0637	Hanger 80°, complete, Pearl White (2015)		2
5	SP1584369-0609	Hanger 80°, complete, Anthracite (2015)		2
6	SP1584369-0515	Hanger 80°, complete, Pearl Grey (2015)		2
7	SP1584381	Kit hanger pivot (2015)		2
8	SP1584383	Kit hanger locking nose (2015)		2
9	SP1584382	Calf strap holder + ball screw (2015)		2
10	1424571	Screw stop M10x23		2
11	SP1584380	Kit hanger locked (2015)		2

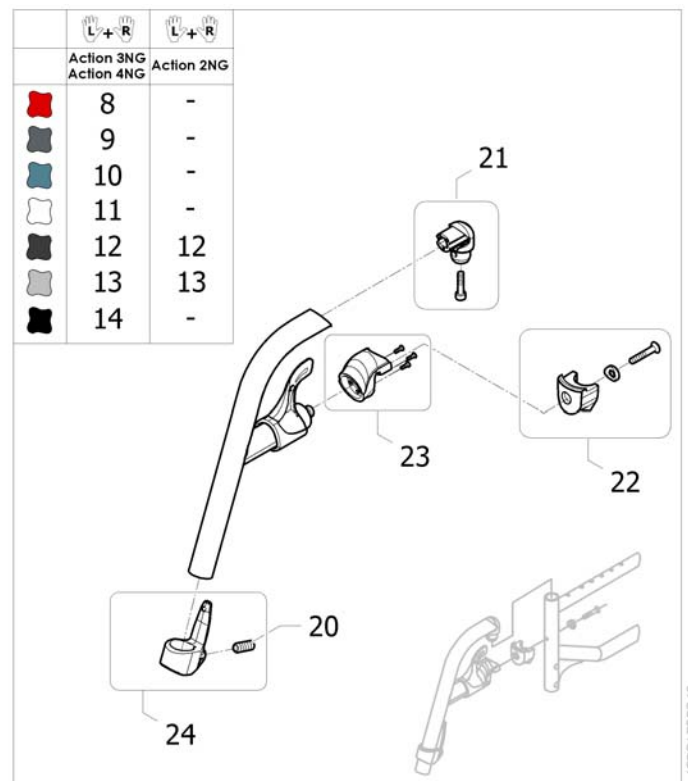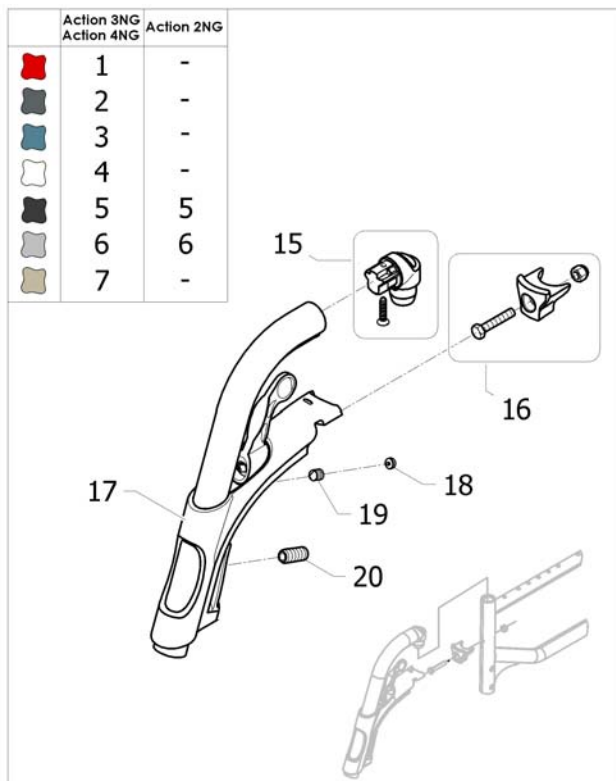Hanger fixed 80° =>2015.01.25


Pos.	Part number	Description	Valid from → until	Qty
1	1428622-0607	Footrest hanger 80° Dark Red		2
2	1428622-0615	Footrest hanger 80° Slate Grey		2
3	1428622-0617	Footrest hanger 80° Ocean Blue		2
4	1428622-0637	Footrest hanger 80° Pearl White		2
5	1428622-0609	Footrest hanger 80° Anthracite		2
6	1428622-0515	Footrest hanger 80° Pearl Grey		2
7	1428622-0616	Footrest hanger 80° Ice Brown	→ 2011-12-31	2
8	SP1584502-0607	Kit Footrest hanger 80° Dark Red Pair (2015 Version)	2015-01-26 →	1
9	SP1584502-0615	Kit Footrest hanger 80° Salte Grey Pair(2015 Version)	2015-01-26 →	1
10	SP1584502-0617	Kit Footrest hanger 80° Océan Blue Pair(2015 Version)	2015-01-26 →	1
11	SP1584502-0637	Kit Footrest hanger 80° Pearl White Pair(2015 Version)	2015-01-26 →	1
12	SP1584502-0609	Kit Footrest hanger 80° Anthracite Pair (2015 Version)	2015-01-26 →	1
13	SP1584502-0515	Kit Footrest hanger 80° Pearl Grey Pair(2015 Version)	2015-01-26 →	1
14	SP1584502-0601	Kit Footrest hanger 80° Black Pair(2015 Version)	2015-01-26 →	1
15	1430426	Hanger pivot kit		2
16	1430427	Kit hanger locked part		2
17	1434362	Hanger support complete (Alu Pin)		2
18	1434070	Key cap Key		2
19	1424572	Screw stop M10x12		2
20	1424571	Screw stop M10x23		2
21	SP1584381	Kit hanger pivot (2015)	2015-01-26 →	2
22	SP1584380	Kit hanger locked (2015)	2015-01-26 →	2
23	SP1584383	Kit hanger locking nose (2015)	2015-01-26 →	2
24	SP1584382	Calf strap holder + ball screw (2015)	2015-01-26 →	2

Assembly Legrest with footrest

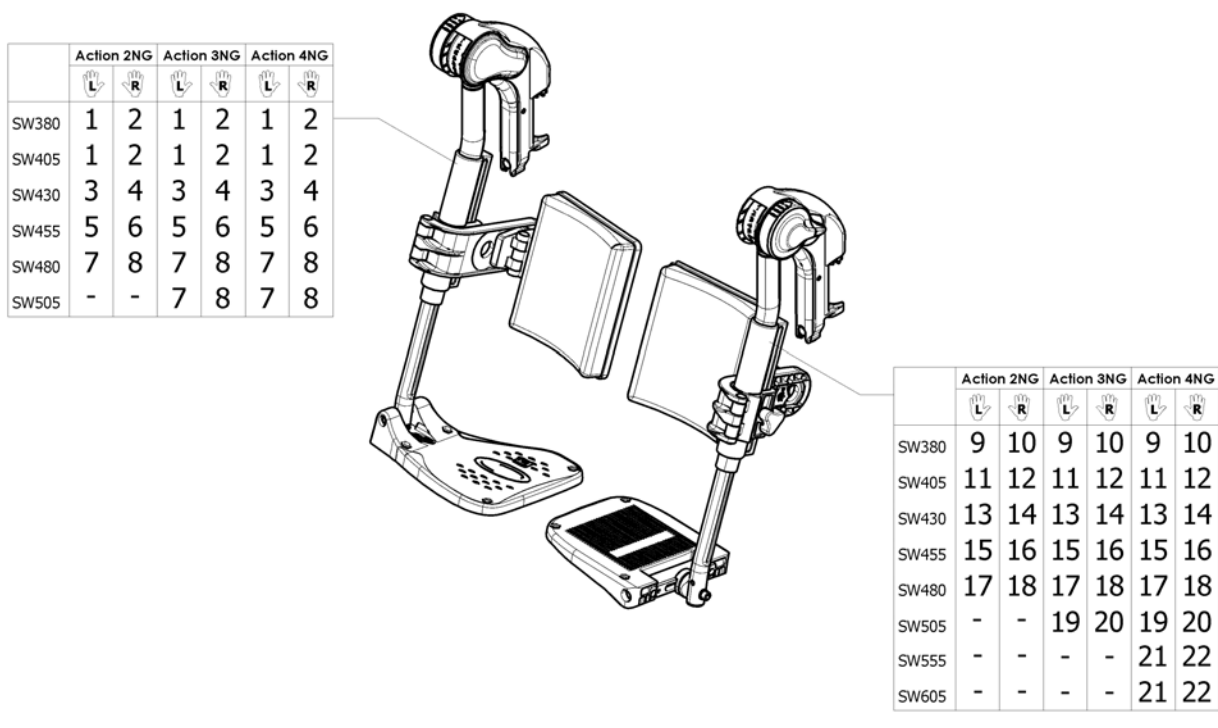

ISPS1535702


Pos.	Part number	Description	Valid from → until	Qty
1	1432974	Lh Elevating Legrest + Fixed Footplate SW380/405 mm		1
2	1432973	Rh Elevating Legrest + Fixed Footplate SW380/405 mm		1
3	1432976	Lh Elevating Legrest + Fixed Footplate SW430 mm		1
4	1432975	Rh Elevating Legrest + Fixed Footplate SW430 mm		1
5	1432978	Lh Elevating Legrest + Fixed Footplate SW455 mm		1
6	1432977	Rh Elevating Legrest + Fixed Footplate SW455 mm		1
8	1432979	Rh Elevating Legrest + Fixed Footplate SW480/505 mm		1
7	1432980	Lh Elevating Legrest + Fixed Footplate SW480/505 mm		1
9	1432982	Lh Elevating Legrest + Adjustable Footplate SW380 mm		1
10	1432981	Rh Elevating Legrest + Adjustable Footplate SW380 mm		1
11	1432984	Lh Elevating Legrest + Adjustable Footplate SW405 mm		1
12	1432983	Rh Elevating Legrest + Adjustable Footplate SW405 mm		1
13	1432986	Lh Elevating Legrest + Adjustable Footplate SW430 mm		1
14	1432985	Rh Elevating Legrest + Adjustable Footplate SW430 mm		1
15	1432948	Lh Elevating Legrest + Adjustable Footplate SW455 mm		1
16	1432947	Rh Elevating Legrest + Adjustable Footplate SW455 mm		1
17	1432964	Lh Elevating Legrest + Adjustable Footplate SW480 mm		1
18	1432949	Rh Elevating Legrest + Adjustable Footplate SW480 mm		1
19	1432971	Lh Elevating Legrest + Adjustable Footplate SW505 mm		1
20	1432965	Rh Elevating Legrest + Adjustable Footplate SW505 mm		1
21	1533603	Lh Elevating Legrest heavy + Adjustable Footplate SW555/605 mm		1
22	1533602	Rh Elevating Legrest heavy + Adjustable Footplate SW555/605 mm		1

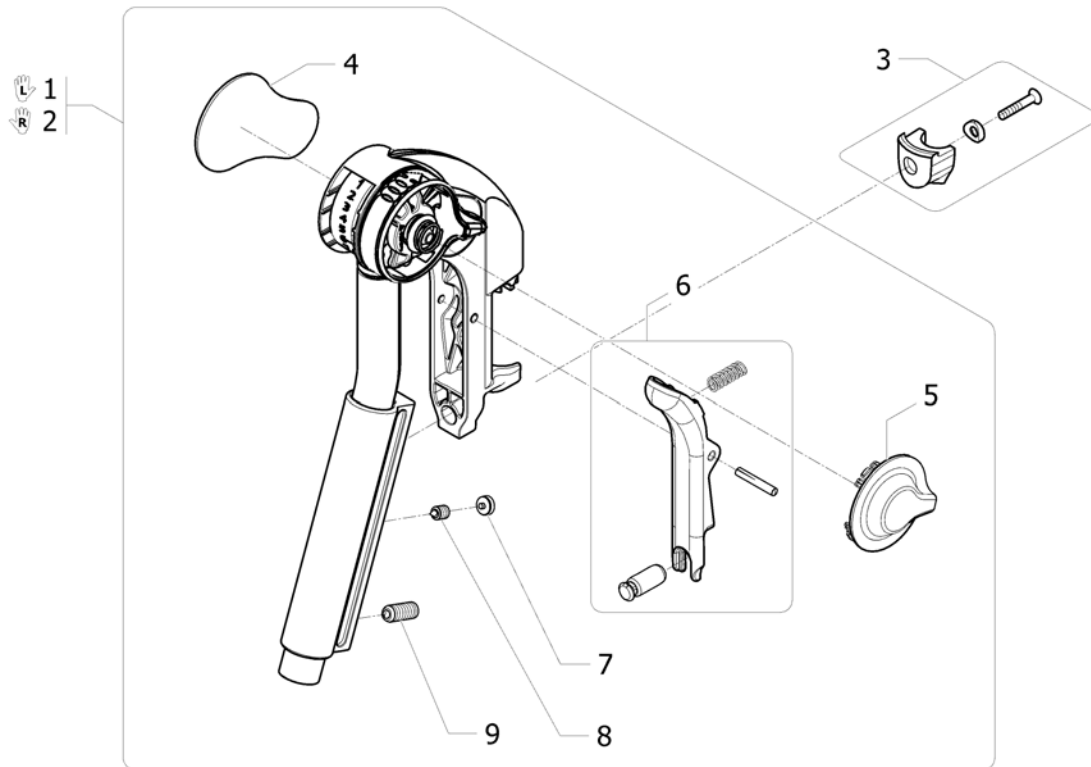Elevating legrest

ISPS1535382


Pos.	Part number	Description	Valid from → until	Qty
1	1552881	Elevating legrest Lh + fixation <i>Replace 1508945</i>		1
2	1552882	Elevating legrest Rh + fixation <i>Replace 1508948</i>		1
3	SP1584380	Kit hanger locked (2015) <i>Replace 1430427</i>		2
4	1428336	Kneepad		2
5	1508943	Incline adjustment lever		2
6	1430429	Legrest Trigger Kit		2
7	1435741	Footrest Hanger Plug		2
8	1424572	Screw stop M10x12		2
9	1424571	Screw stop M10x23		2

Calf Pad

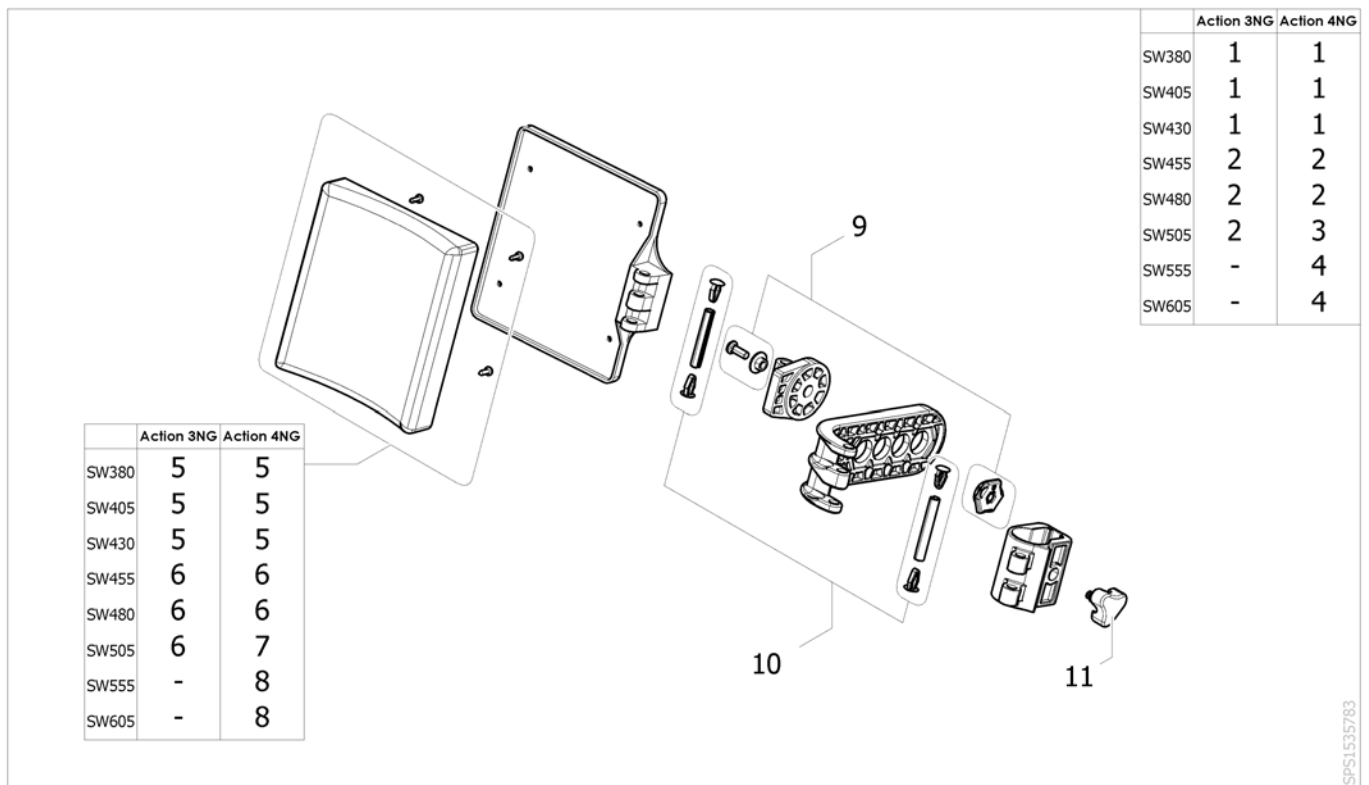

Pos.	Part number	Description	Valid from → until	Qty
1	1450932	Complete Polyurethane calf pad (cushion with support), SW380/405/430 mm		2
2	1450933	Complete Polyurethane calf pad (cushion with support), SW455/480/505 mm		2
3	1450935	Complete Dartex calf pad (cushion with support), SW480/505 mm		2
4	1450936	Complete Dartex calf pad (cushion with support), SW555/605 mm		2
5	1430430	Polyurethane calf pad cushion with fixation SW380/405/430 mm		2
6	1430431	Polyurethane calf pad cushion with fixation SW450/480/505 mm		2
7	1442646	Dartex calf pad cushion with fixation, SW480/505 mm		2
8	1532965	Dartex calf pad cushion with fixation, SW555/605 mm		2
9	1440132	Kit fixation angle adjustable calf-pad		2
10	1430432	Kit articulation pin calf pad		2
11	1440943	Locking star knob		2


Amputee Legrest

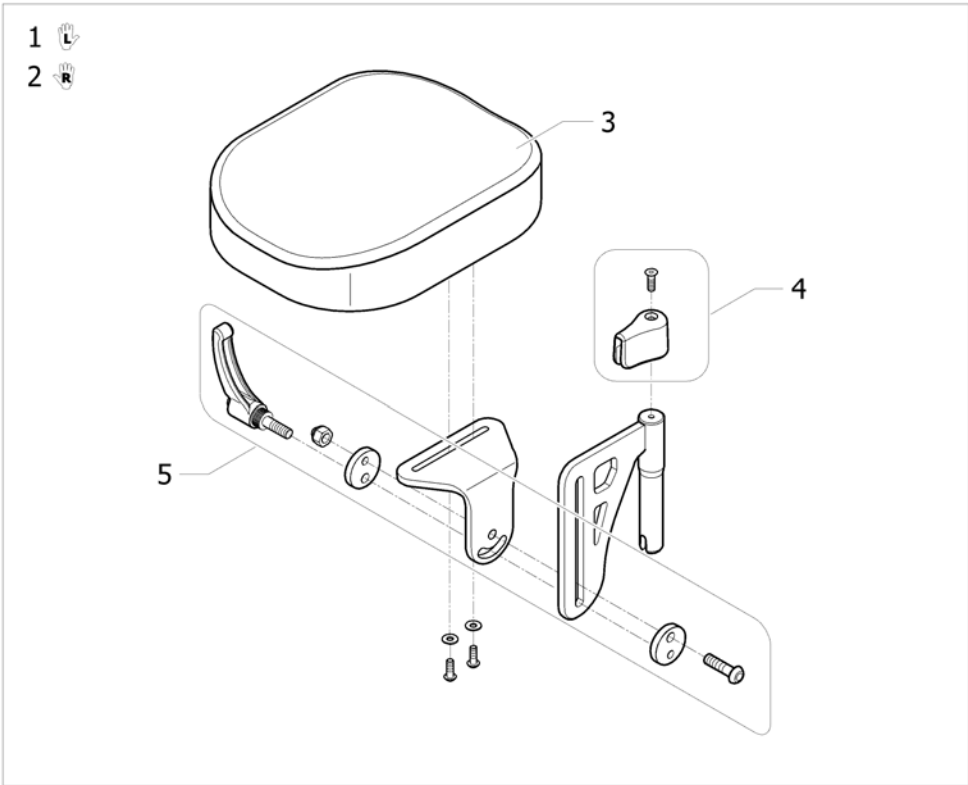

ISPS1495897

Pos.	Part number	Description	Valid from → until	Qty
1	1442991	Support Lh, legrest amputee		1
2	1442990	Support Rh, legrest amputee		1
3	1497450	Cushion amputee legrest		2
4	1446556	Amputee legrest stop + fixation		2
5	1446557	Amputee legrest slide fixation		2
6	SP1584380	Kit hanger locked (2015) <i>Replace 1430427</i>		2

Footrests

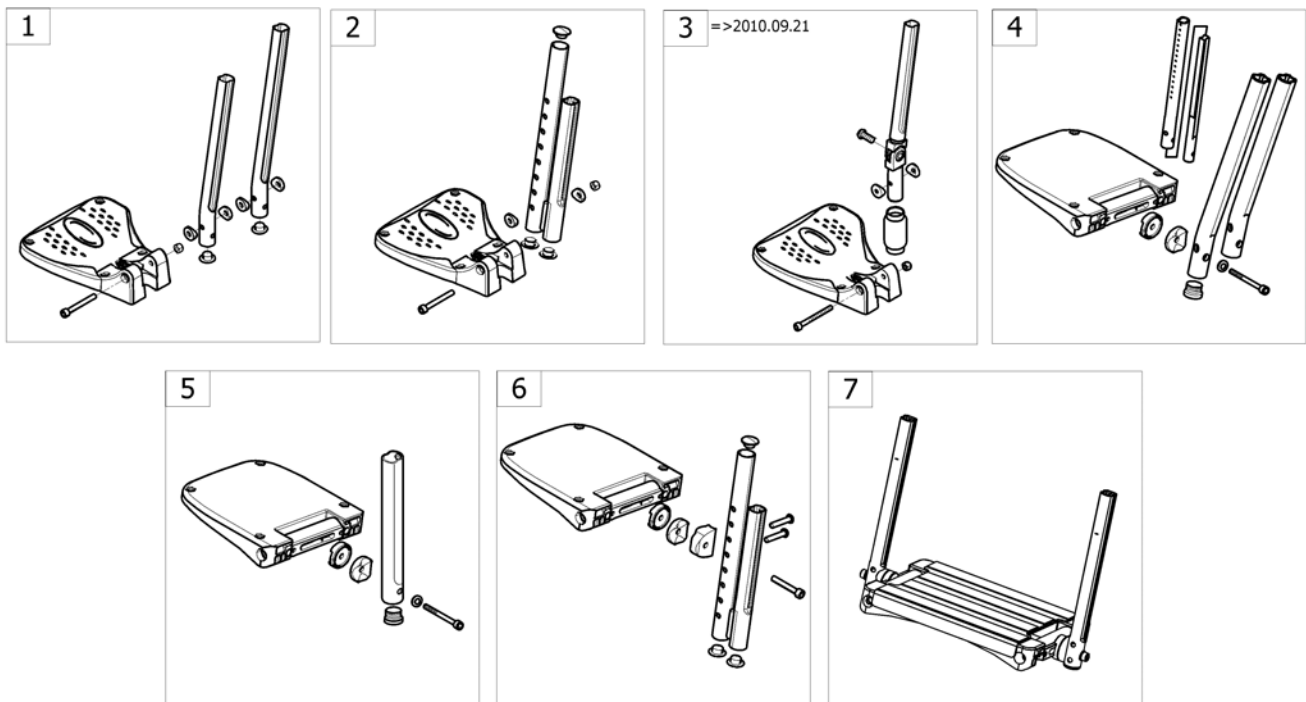


SPCACTION4NG35

Pos.	Part number	Description	Valid from → until	Qty
1	-	Fixed footrest		1
2	-	Fixed footrest with shorter support		1
3	-	Angle adjustable footrest =>2010.09.21	→ 2010-09-21	1
4	-	Angle and depth adjustable footrest		1
5	-	Angle and depth adjustable footrest SH360 mm		1
6	-	Angle and depth adjustable footrest with shorter support		1
7	-	One piece footboard		1

Fixed footrest

	Action 3NG Action 2000				Action 4NG			
								
								
SW380	1	2	9	10	1	2	9	10
SW405	1	2	9	10	1	2	9	10
SW430	3	4	11	12	3	4	11	12
SW455	3	4	11	12	3	4	11	12
SW480	5	6	13	14	5	6	13	14
SW505	5	6	13	14	5	6	13	14
SW555	-	-	-	-	7	8	15	16
SW605	-	-	-	-	7	8	15	16

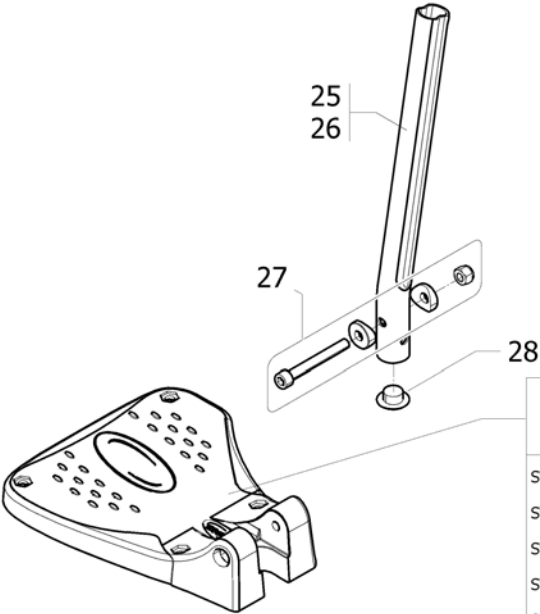


	Action 3NG Action 2000		Action 4NG	
				
SW380	17	18	17	18
SW405	17	18	17	18
SW430	19	20	19	20
SW455	19	20	19	20
SW480	21	22	21	22
SW505	21	22	21	22
SW555	-	-	23	24
SW605	-	-	23	24

163515391

Pos.	Part number	Description	Valid from → until	Qty
1	1530620	Footrest fixed, Lh, SW380-405mm, standard tube, swing away frame		1
2	1530614	Footrest fixed, Rh, SW380-405mm, standard tube, swing away frame		1
3	1530622	Footrest fixed, Lh, SW430-455mm, standard tube, swing away frame		1
4	1530616	Footrest fixed, Rh, SW430-455mm, standard tube, swing away frame		1
5	1530623	Footrest fixed, Lh, SW480-505mm, standard tube, swing away frame		1
6	1530617	Footrest fixed, Rh, SW480-505mm, standard tube, swing away frame		1
7	1488624	Kit footrest fixed short & bended tube Lh(SW550/605 Action NG)(SW490 Esprit Clematis)		1
8	1488623	Kit footrest fixed short & bended tube Rh(SW550/605 Action NG)(SW490 Esprit Clematis)		1
9	1530633	Footrest fixed, Lh, SW380-405mm, long tube, swing away frame		1
10	1530630	Footrest fixed, Rh, SW380-405mm, long tube, swing away frame		1
11	1530634	Footrest fixed, Lh, SW330-455mm, long tube, swing away frame		1
12	1530631	Footrest fixed, Rh, SW330-455mm, long tube, swing away frame		1
13	1530635	Footrest fixed, Lh, SW480-505mm, long tube, swing away frame		1
14	1530632	Footrest fixed, Rh, SW480-505mm, long tube, swing away frame		1
15	1488626	Lh Standard Footrest SW555/605 mm - Long Tube		1
16	1488625	Rh Standard Footrest SW555/605 mm - Long Tube		1
17	1529643	Footplate, fixed, Lh, 174mm (V2009)		1
18	1529642	Footplate, fixed, Rh, 174mm (V2009)		1
19	1529645	Footplate, fixed, Lh, 199mm (V2009)		1
20	1529644	Footplate, fixed, Rh, 199mm (V2009)		1
21	1529647	Footplate, fixed, Lh, 224mm (V2009)		1
22	1529646	Footplate, fixed, Rh, 224mm (V2009)		1
23	1428374	Fixed footplate NG Lh 250 mm (SW550/605 Action NG)(SW490 Esprit Clematis)		1
24	1428375	Fixed footplate NG Rh 250 mm (SW550/605 Action NG)(SW490 Esprit Clematis)		1
25	1558855-0010	Standard footrest tube for a&d adjustable footrest		2
26	1434563-0010	Long tube for fixed and a&d adjustable footrest		2
27	1430433	Kit screws for fixed footrest		2
28	1424576	Plastic cap black Ø20mm		2

Fixed footrest with shorter support

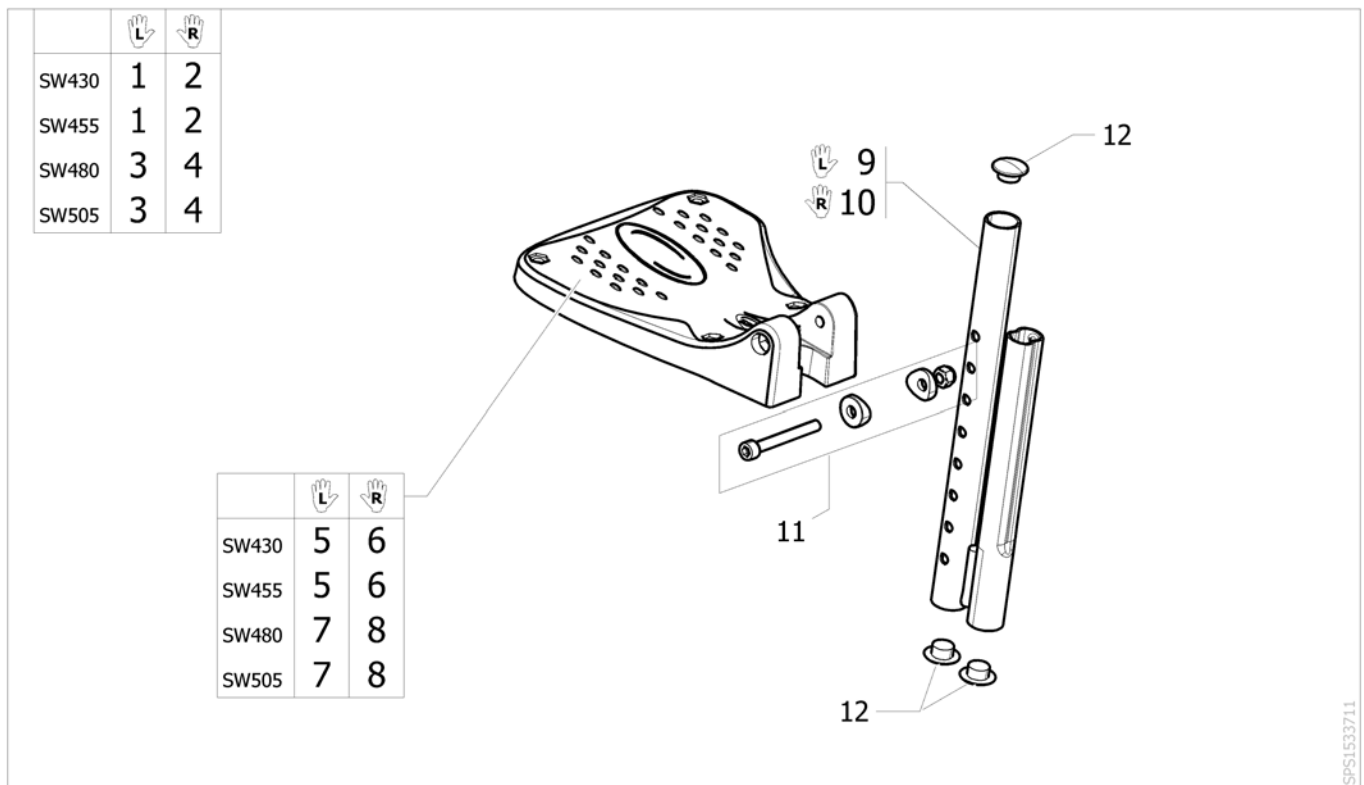

Pos.	Part number	Description	Valid from → until	Qty
1	1532070	Footrest fixed, Lh, SW430-455mm, high mounted tube		1
2	1532066	Footrest fixed, Rh, SW430-455mm, high mounted tube		1
3	1532072	Footrest fixed, Lh, SW480-505mm, high mounted tube		1
4	1532068	Footrest fixed, Rh, SW480-505mm, high mounted tube		1
5	1529643	Footplate, fixed, Lh, 174mm (V2009) <i>Replace 1428340</i>		1
6	1529642	Footplate, fixed, Rh, 174mm (V2009) <i>Replace 1428341</i>		1
7	1529645	Footplate, fixed, Lh, 199mm (V2009) <i>Replace 1424574</i>		1
8	1529644	Footplate, fixed, Rh, 199mm (V2009) <i>Replace 1424575</i>		1
9	1531202-0699	Footrest tube high mounted Lh		1
10	1531203-0699	Footrest tube high mounted Rh		1
11	1430433	Kit screws for fixed footrest		2
12	1424576	Plastic cap black Ø20mm		6

Angle adjustable footrest =>2010.09.21


	Action 3NG		Action 4NG	
	L	R	L	R
SW380	1	2	1	2
SW405	1	2	1	2
SW430	3	4	3	4
SW455	3	4	3	4
SW480	5	6	5	6
SW505	5	6	5	6
SW555	-	-	7	8
SW605	-	-	7	8

	Action 3NG		Action 4NG	
	L	R	L	R
SW380	9	10	9	10
SW405	9	10	9	10
SW430	11	12	11	12
SW455	11	12	11	12
SW480	13	14	13	14
SW505	13	14	13	14
SW555	-	-	15	16
SW605	-	-	15	16

ISPS1535392

Pos.	Part number	Description	Valid from → until	Qty
1	1530640	Lh Adjustable Footrest SW380/405 mm	→ 2010-09-21	1
2	1530637	Rh Adjustable Footrest SW380/405 mm	→ 2010-09-21	1
3	1530641	Lh Adjustable Footrest SW430/455 mm	→ 2010-09-21	1
4	1530638	Rh Adjustable Footrest SW430/455 mm	→ 2010-09-21	1
5	1530642	Lh Adjustable Footrest SW480/505 mm	→ 2010-09-21	1
6	1530639	Rh Adjustable Footrest SW480/505 mm	→ 2010-09-21	1
7	1485227	Lh Adjustable Footrest SW555/605 mm - Short Tube	→ 2010-09-21	1
8	1485226	Rh Adjustable Footrest SW555/605 mm - Short Tube	→ 2010-09-21	1
9	1529643	Footplate, fixed, Lh, 174mm (V2009)	→ 2010-09-21	1
10	1529642	Footplate, fixed, Rh, 174mm (V2009)	→ 2010-09-21	1
11	1529645	Footplate, fixed, Lh, 199mm (V2009)	→ 2010-09-21	1
12	1529644	Footplate, fixed, Rh, 199mm (V2009)	→ 2010-09-21	1
13	1529647	Footplate, fixed, Lh, 224mm (V2009)	→ 2010-09-21	1
14	1529646	Footplate, fixed, Rh, 224mm (V2009)	→ 2010-09-21	1
15	1428374	Fixed footplate NG Lh 250 mm (SW550/605 Action NG)(SW490 Esprit Clematis)	→ 2010-09-21	1
16	1428375	Fixed footplate NG Rh 250 mm (SW550/605 Action NG)(SW490 Esprit Clematis)	→ 2010-09-21	1
17	1485167	Short Footrest Tube	→ 2010-09-21	2
17.1	0029-40046	Screw CBHC M8X20	→ 2010-09-21	2
17.2	1485944	Adjustable Footrest Sleeve	→ 2010-09-21	2
18	1430433	Kit screws for fixed footrest	→ 2010-09-21	2

Angle and depth adjustable footrest

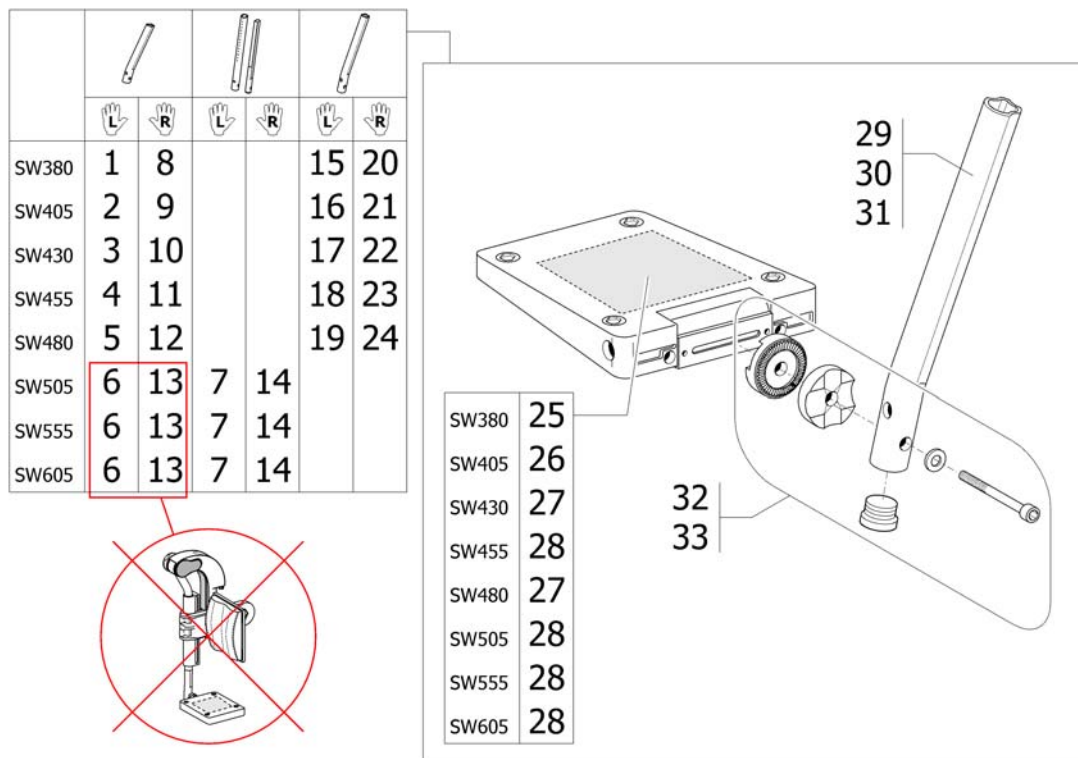

Pos.	Part number	Description	Valid from → until	Qty
1	1428657	Footrest a&d adjustable, Lh, SW380mm, standard tube, swing away frame		1
2	1428658	Footrest a&d adjustable, Lh, SW405mm, standard tube, swing away frame		1
3	1428659	Footrest a&d adjustable, Lh, SW430mm, standard tube, swing away frame		1
4	1428660	Footrest a&d adjustable, Lh, SW455mm, standard tube, swing away frame		1
5	1428661	Footrest a&d adjustable, Lh, SW480mm, standard tube, swing away frame		1
6	1428662	Lh angle and depth adjustable footrest short tube SW505-555-605 (Clematis)		1
7	1532964	Footrest a&d adjustable, Lh, SW505mm, standard tube, swing away frame		1
8	1428651	Footrest a&d adjustable, Rh, SW380mm, standard tube, swing away frame		1
9	1428652	Footrest a&d adjustable, Rh, SW405mm, standard tube, swing away frame		1
10	1428653	Footrest a&d adjustable, Rh, SW430mm, standard tube, swing away frame		1
11	1428654	Footrest a&d adjustable, Rh, SW455mm, standard tube, swing away frame		1
12	1428655	Footrest a&d adjustable, Rh, SW480mm, standard tube, swing away frame		1
13	1428656	Rh angle and depth adjustable footrest short tube(SW505/605 Action		1
14	1532963	Footrest a&d adjustable, Rh, SW505mm, standard tube, swing away frame		1
15	1440816	Footrest a&d adjustable, Lh, SW380mm, long tube, swing away frame		1
16	1440817	Footrest a&d adjustable, Lh, SW405mm, long tube, swing away frame		1
17	1440818	Footrest a&d adjustable, Lh, SW430mm, long tube, swing away frame		1
18	1440819	Footrest a&d adjustable, Lh, SW455mm, long tube, swing away frame		1
19	1440820	Footrest a&d adjustable, Lh, SW480mm, long tube, swing away frame		1
20	1440810	Footrest a&d adjustable, Rh, SW380mm, long tube, swing away frame		1
21	1440811	Footrest a&d adjustable, Rh, SW405mm, long tube, swing away frame		1
22	1440812	Footrest a&d adjustable, Rh, SW430mm, long tube, swing away frame		1
23	1440813	Footrest a&d adjustable, Rh, SW455mm, long tube, swing away frame		1
24	1440814	Footrest a&d adjustable, Rh, SW480mm, long tube, swing away frame		1
25	5106374	Footplate, angle and depth adjustable, 141 mm		2
26	5106375	Footplate, angle and depth adjustable, 156 mm		2

Angle and depth adjustable footrest

Pos.	Part number	Description	Valid from → until	Qty
27	5106594	Footplate, angle and depth adjustable, 166 mm		2
28	5106376	Footplate, angle and depth adjustable, 176 mm		2
29	1558855-0010	Standard footrest tube for a&d adjustable footrest		2
30	1533719	Standard footrest tube with heavy bar		2
31	1434563-0010	Long tube for fixed and a&d adjustable footrest		2
32	1430839	Kit fixation, adjustable footplate, short spacer		2
33	1430840	Kit fixation, adjustable footplate, long spacer		2

Angle and depth adjustable footrest SH360 mm


	Action 3NG		Action 4NG	
	L	R	L	R
SW380	1	7	1	7
SW405	2	8	2	8
SW430	3	9	3	9
SW455	4	10	4	10
SW480	5	11	5	11
SW505	6	12	6	12
SW555	-	-	6	12
SW605	-	-	6	12

	Action 3NG	Action 4NG
SW380	13	13
SW405	14	14
SW430	15	15
SW455	16	16
SW480	15	15
SW505	16	16
SW555	-	16
SW605	-	16

SPS1548903 ~ sps1497630_II

Pos.	Part number	Description	Valid from → until	Qty
1	1548477	Footrest a&d adjustable, Lh, SW380mm, short tube, swing away frame		1
2	1548487	Footrest a&d adjustable, Lh, SW405mm, short tube, swing away frame		1
3	1548489	Footrest a&d adjustable, Lh, SW430mm, short tube, swing away frame		1
4	1548491	Footrest a&d adjustable, Lh, SW455mm, short tube, swing away frame		1
5	1548518	Footrest a&d adjustable, Lh, SW480mm, short tube, swing away frame		1
6	1548520	Footrest a&d adjustable, Lh, SW505mm, short tube, swing away frame		1
7	1548478	Footrest a&d adjustable, Rh, SW380mm, short tube, swing away frame		1
8	1548488	Footrest a&d adjustable, Rh, SW405mm, short tube, swing away frame		1
9	1548490	Footrest a&d adjustable, Rh, SW430mm, short tube, swing away frame		1
10	1548492	Footrest a&d adjustable, Rh, SW455mm, short tube, swing away frame		1
11	1548519	Footrest a&d adjustable, Rh, SW480mm, short tube, swing away frame		1
12	1548521	Footrest a&d adjustable, Rh, SW505mm, short tube, swing away frame		1
13	5106374	Footplate, angle and depth adjustable, 141 mm		2
14	5106375	Footplate, angle and depth adjustable, 156 mm		2
15	5106594	Footplate, angle and depth adjustable, 166 mm		2
16	5106376	Footplate, angle and depth adjustable, 176 mm		2
17	1541994	Short footrest tube for a&d adjustable footrest		2
18	1430839	Kit fixation, adjustable footplate, short spacer		2
19	1430840	Kit fixation, adjustable footplate, long spacer		2


Angle and depth adjustable footrest with shorter support

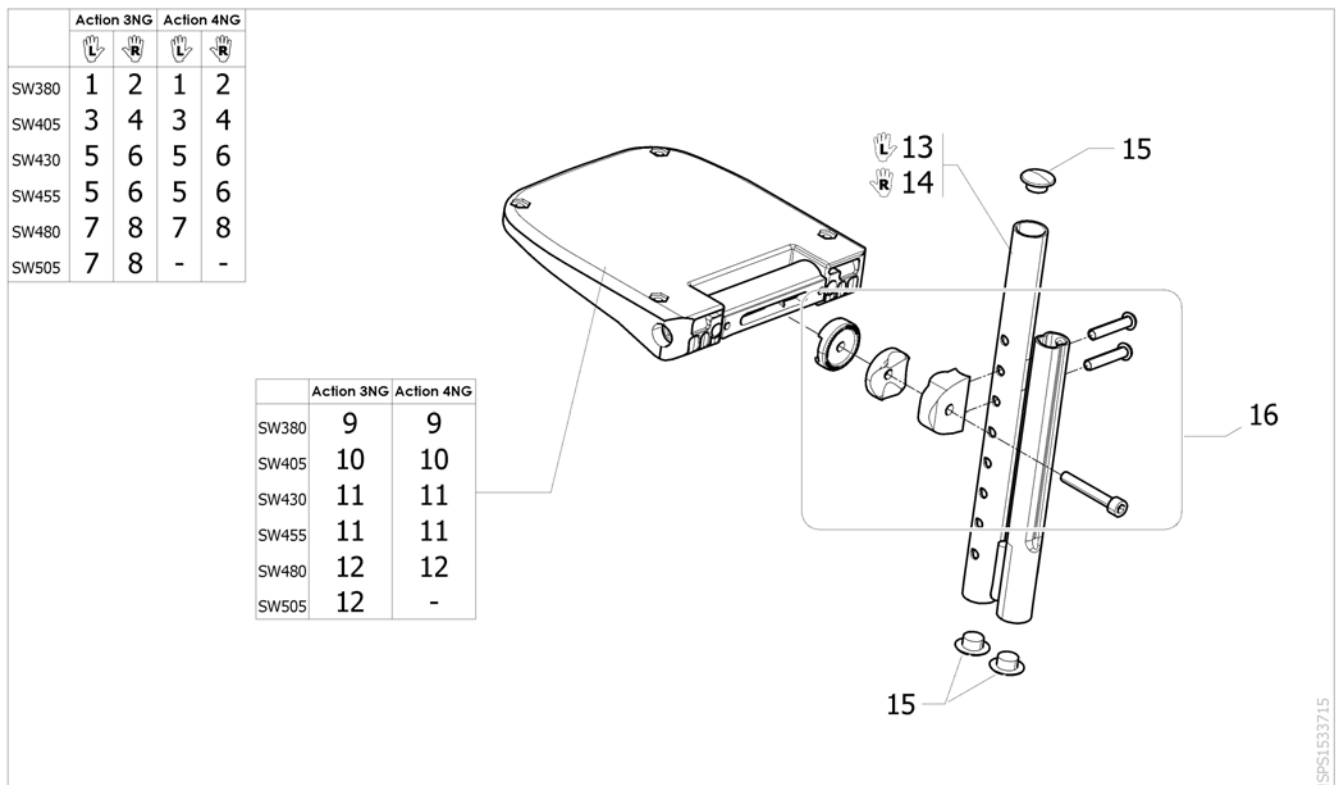

Pos.	Part number	Description	Valid from → until	Qty
1	SP1586054	Footrest a&d adjustable, Lh, SW380mm, high mounted tube	2015-01-01 →	1
2	SP1586052	Footrest a&d adjustable, Rh, SW380mm, high mounted tube	2015-01-01 →	1
3	SP1586068	Footrest a&d adjustable, Lh, SW405mm, high mounted tube	2015-01-01 →	1
4	SP1586053	Footrest a&d adjustable, Rh, SW405mm, high mounted tube	2015-01-01 →	1
5	1532078	Footrest a&d adjustable, Lh, SW430-455mm, high mounted tube		1
6	1532074	Footrest a&d adjustable, Rh, SW430-455mm, high mounted tube		1
7	1532080	Footrest a&d adjustable, Lh, SW480-505mm, high mounted tube		1
8	1532076	Footrest a&d adjustable, Rh, SW480-505mm, high mounted tube		1
9	5106371	Footplate, angle and depth adjustable, 106 mm		2
10	5106373	Footplate, angle and depth adjustable, 131 mm		2
11	5106374	Footplate, angle and depth adjustable, 141 mm		2
12	5106594	Footplate, angle and depth adjustable, 166 mm		2
13	1531202-0699	Footrest tube high mounted Lh		1
14	1531203-0699	Footrest tube high mounted Rh		1
15	1424576	Plastic cap black Ø20mm		6
16	1533714	Kit fixation for angle & depth adjustable footrest, high mounted tube		2


One piece footboard

	Action 3NG	Action 4NG
SW380	1	1
SW405	2	2
SW430	3	3
SW455	4	4
SW480	5	5
SW505	6	6
SW555	-	7
SW605	-	8

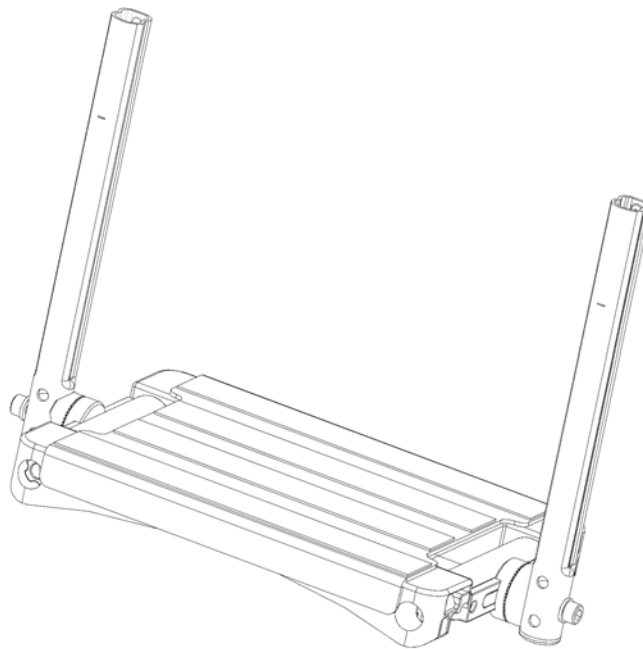

15P51514296


Pos.	Part number	Description	Valid from → until	Qty
1	1496945	One piece footboard, short, SW380 mm, depth and angle adjustable, hanger 80°		1
2	1496946	One piece footboard, short, SW405 mm, depth and angle adjustable, hanger 80°		1
3	1496947	One piece footboard, short, SW430 mm, depth and angle adjustable, hanger 80°		1
4	1496948	One piece footboard, short, SW455 mm, depth and angle adjustable, hanger 80°		1
5	1496949	One piece footboard, short, SW480 mm, depth and angle adjustable, hanger 80°		1
6	1496953	One piece footboard, short, SW505 mm, depth and angle adjustable, hanger 80°		1
7	1510382	One piece footboard, short, SW555 mm, depth and angle adjustable, hanger 80°		1
8	1510383	One piece footplate SW605 mm		1

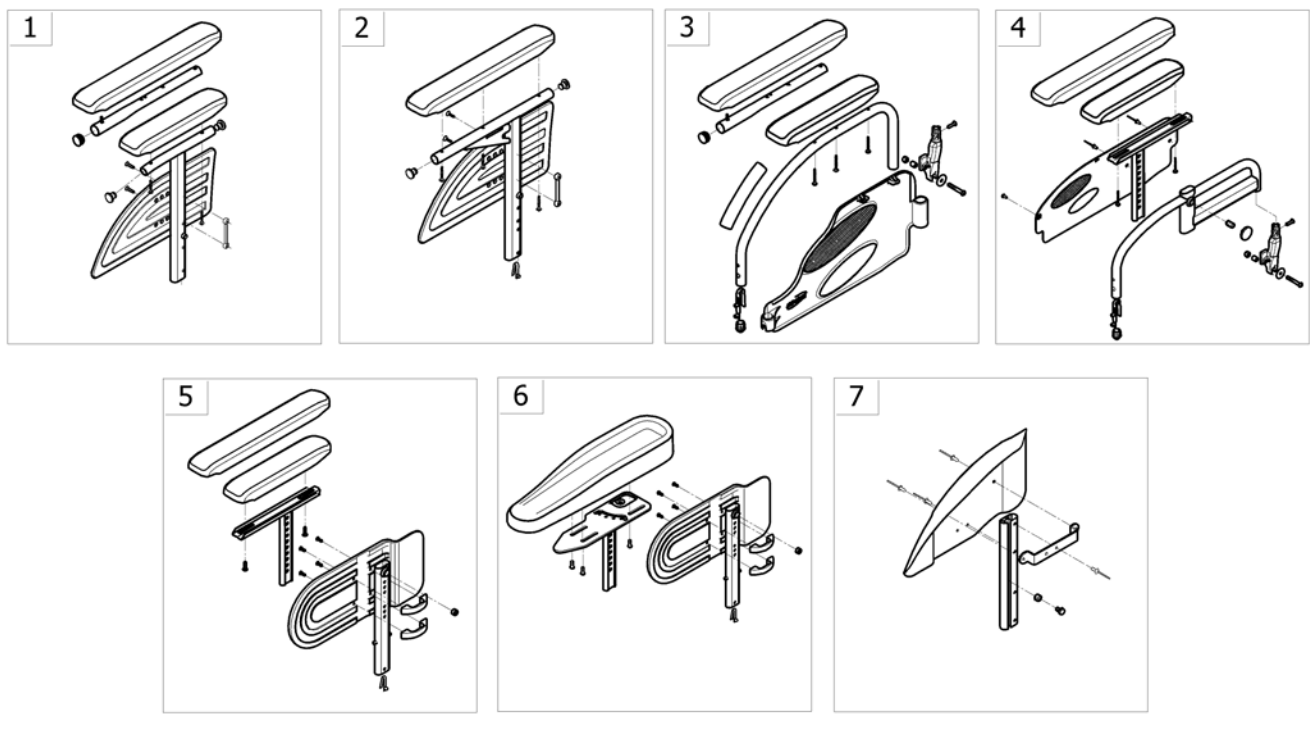Siderests

<SPCACTION4NG40

Pos.	Part number	Description	Valid from → until	Qty
1	-	Height Adjustable Armrest 1 Rio [CK_0520/CK_0530/CK_0580/CK_0590]		1
2	-	T armrest n°1 Rio - Renouvelable and height adjustable [CK_0520/CK_0580]		1
3	-	Swing Away & Detachable Armrest 2 [CK_0537/CK_0538/CK_0597/CK_0598]		1
4	-	Height adjustable armrest n°3 [CK_0539/CK_0540/CK_0599/CK_0600]		1
5	-	Height adjustable armrest n°4 [CK_0533/CK_0536/CK_0593/CK_0596]		1
6	-	Hemi Dartex armpad n°4 multiposition standard [CK_0555/CK_0615]		1
7	-	Mudguard Xtra [CK_0556/CK_0616]		1


Height Adjustable Armrest 1 Rio

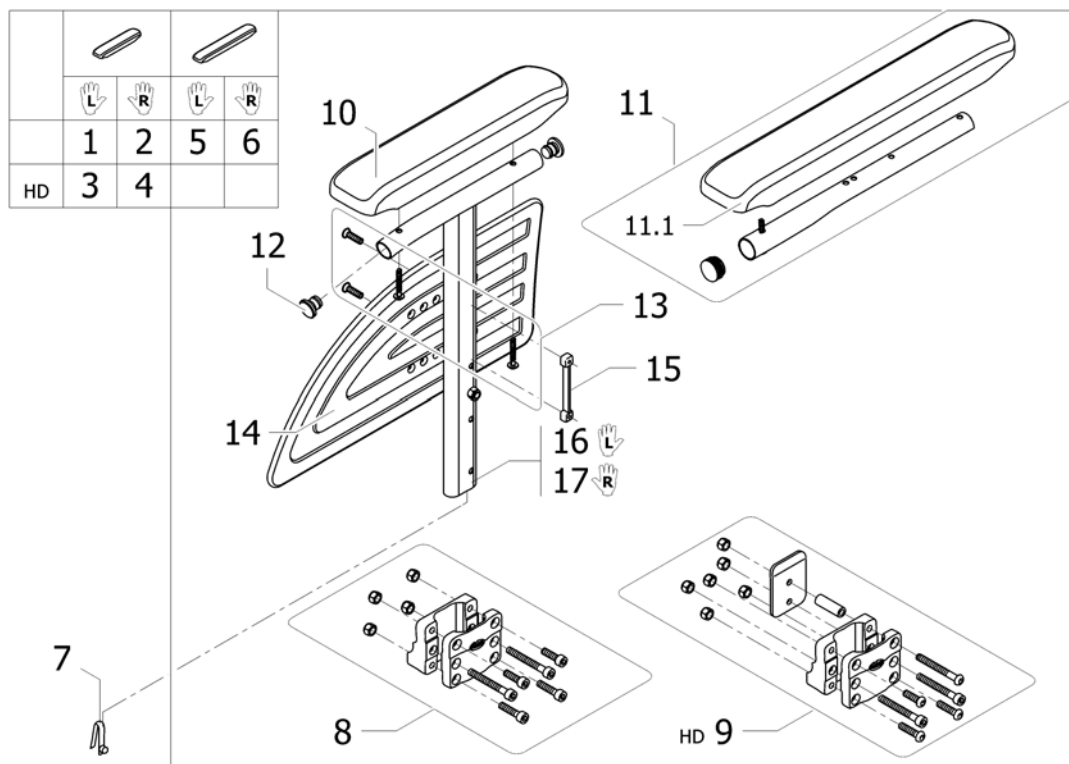

ISPS1496337 ~ sps1495339_jl


Pos.	Part number	Description	Valid from → until	Qty
1	1449978	Lh Desk Armrest N°1; Removable And Height Adjustable; Rio [CK_0580]		1
2	1449977	Rh Desk Armrest N°1; Removable And Height Adjustable; Rio [CK_0520]		1
3	1478771	Lh Desk Armrest N°1 Rio Hd [HD+CK_0580]		1
4	1478770	Rh Desk Armrest N°1 Rio Hd [HD+CK_0520]		1
5	1449980	Lh Long Armrest N°1; Removable And Height Adjustable; Rio [CK_0590]		1
6	1449979	Rh Long Armrest N°1; Removable And Height Adjustable; Rio [CK_0530]		1
7	0027-10003	Spring pin locked armrest Replace 0027-10002		2
8	1430421	Central armrest support		2
9	1532575	Central Support Armrest 1 Kit HD Replace 1487436		2
10	1543220	Kit short armpad 280mm + 2 screws Replace 1426721		2
11	1432595	Long armpad kit + tube + screw		2
11.1	1543247	Kit armpad long 380 mm + 3 screws Replace 1426723		2
12	0021-20028	Plastic cap black Ø19x15mm		4
13	1430422	Armrest N°1 screws kit		2
14	1434725	Skirtguard Rio height adjustable		2
15	1435388	double-nut		2
16	1425467-0601	Lh Armrest Tube Support		1
17	1428721-0601	Rh Armrest Tube Support		1

T armrest n°1 Rio - Renouvelable and height adjustable

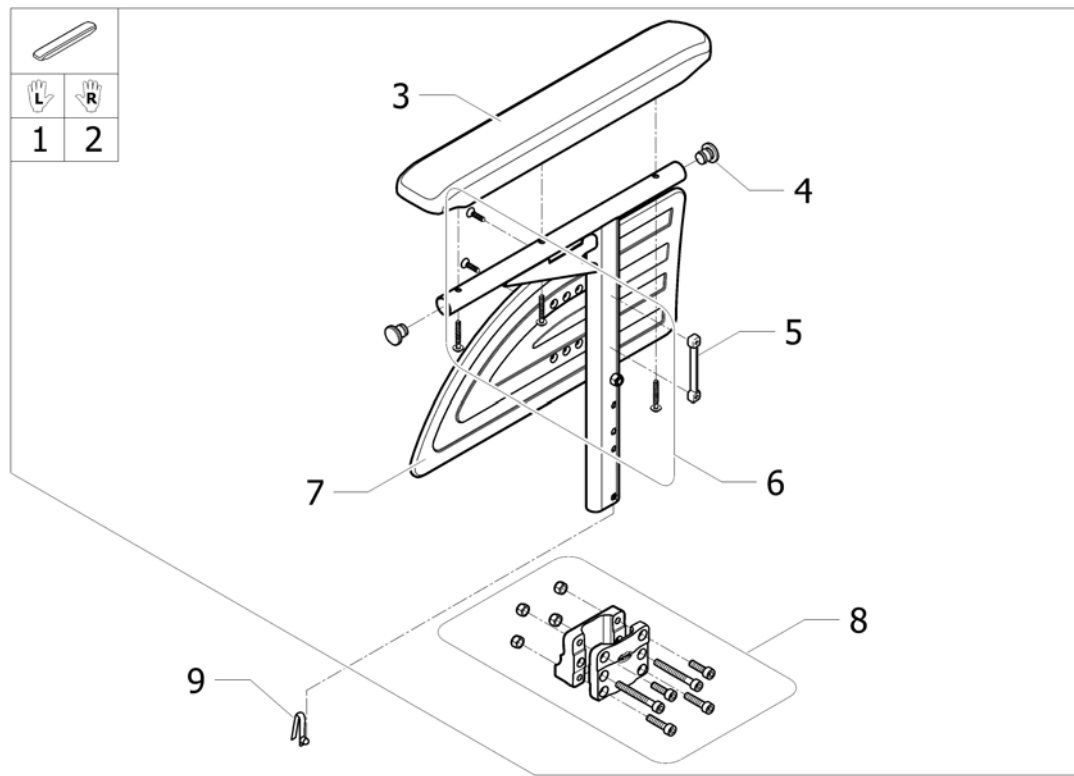

Pos.	Part number	Description	Valid from → until	Qty
1	1554352	Lh T Armrest N°1 rio long - Removable and height adjustable for UK [CK_0580]		1
2	1554353	Rh T Armrest N°1 rio long - Removable and height adjustable for UK [CK_0520]		1
3	1543247	Kit armpad long 380 mm + 3 screws		2
4	0021-20028	Plastic cap black Ø19x15mm		4
5	1435388	double-nut		2
6	1430422	Armrest N°1 screws kit		2
7	1434725	Skirtguard Rio height adjustable		2
8	1430421	Central armrest support		2
9	0027-10003	Spring pin locked armrest		2

Swing Away & Detachable Armrest 2

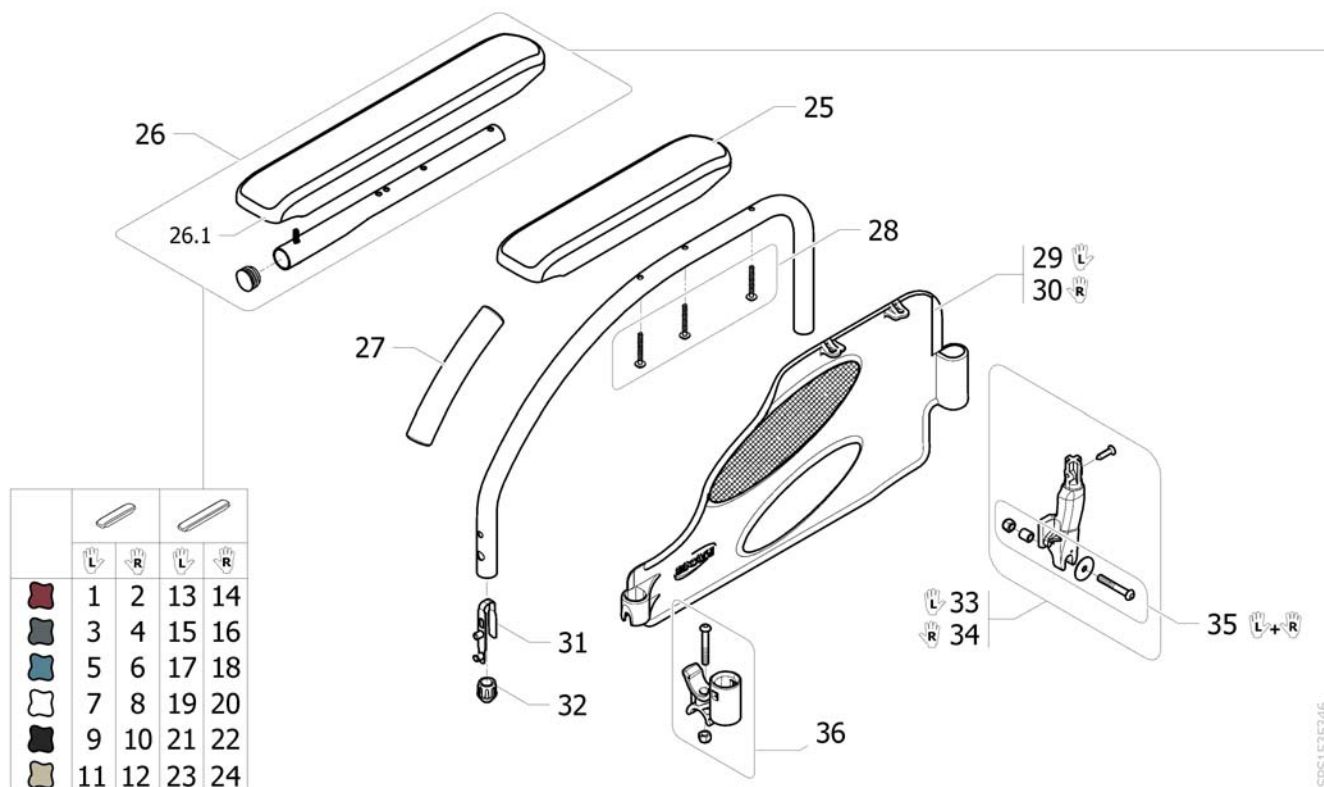


Pos.	Part number	Description	Valid from → until	Qty
1	1428605-0607	Lh Desk Armrest N°2 Dark Red [CE_0760][CK_0597]		1
2	1428604-0607	Rh Desk Armrest N°2 Dark Red [CE_0660][CK_0537]		1
3	1428605-0615	Lh Desk Armrest N°2 Slate Grey [CE_0760][CK_0597]		1
4	1428604-0615	Rh Desk Armrest N°2 Slate Grey [CE_0660][CK_0537]		1
5	1428605-0617	Lh Desk Armrest N°2 Ocean Blue [CE_0760][CK_0597]		1
6	1428604-0617	Rh Desk Armrest N°2 Ocean Blue [CE_0660][CK_0537]		1
7	1428605-0637	Lh Desk Armrest N°2 Pearl White [CE_0760][CK_0597]		1
8	1428604-0637	Rh Desk Armrest N°2 Pearl White [CE_0660][CK_0537]		1
9	1428605-0609	Lh Desk Armrest N°2 Anthracite Black [CE_0760][CK_0597]		1
10	1428604-0609	Rh Desk Armrest N°2 Anthracite Black [CE_0660][CK_0537]		1
11	1428605-0616	Lh Desk Armrest N°2 Ice Brown [CE_0760][CK_0597]	→ 2011-12-31	1
12	1428604-0616	Rh Desk Armrest N°2 Ice Brown [CE_0660][CK_0537]	→ 2011-12-31	1
13	1428607-0607	Lh Long Armrest N°2 Dark Red [CE_0770][CK_0598]		1
14	1428606-0607	Rh Long Armrest N°2 Dark Red [CE_0670][CK_0538]		1
15	1428607-0615	Lh Long Armrest N°2 Slate Grey [CE_0770][CK_0598]		1
16	1428606-0615	Rh Long Armrest N°2 Slate Grey [CE_0670][CK_0538]		1
17	1428607-0617	Lh Long Armrest N°2; Ocean Blue [CE_0770][CK_0598]		1
18	1428606-0617	Rh Long Armrest N°2 Ocean Blue [CE_0670][CK_0538]		1

Swing Away & Detachable Armrest 2

Pos.	Part number	Description	Valid from → until	Qty
19	1428607-0637	Lh Long Armrest N°2 Pearl White <i>[CE_0770][CK_0598]</i>		1
20	1428606-0637	Rh Long Armrest N°2 Pearl White <i>[CE_0670][CK_0538]</i>		1
21	1428607-0609	Lh Long Armrest N°2 Anthracite Black <i>[CE_0770][CK_0598]</i>		1
22	1428606-0609	Rh Long Armrest N°2 Anthracite Black <i>[CE_0670][CK_0538]</i>		1
23	1428607-0616	Lh Long Armrest N°2 Ice Brown <i>[CE_0770][CK_0598]</i>	→ 2011-12-31	1
24	1428606-0616	Rh Long Armrest N°2 Ice Brown <i>[CE_0670][CK_0538]</i>	→ 2011-12-31	1
25	1543195	Kit armpad short 280 mm + 3 screws <i>Replace 1426721</i>		2
26	1432595	Long armpad kit + tube + screw		2
26.1	1543219	Kit armpad long 380 mm + 4 screws <i>Replace 1426723</i>		2
27	0572-02851	Armrest handle cover		2
28	1430424	Armrest 2 Screws Kit N°2		2
29	1491220	Lh armrest skirtguard 2 <i>Replace 1532799</i>		1
30	1491219	Rh armrest skirtguard 2 <i>Replace 1532800</i>		1
31	1018225	Double stud spring		2
32	0572-03405	Short conical cap		2
33	1431284	Lh Armrest Rear Support Kit		1
34	1430423	Rh Armrest Rear Support Kit		1
35	1573039	Fixed armrest N°2/3 fixation kit (pair)		1
36	0572-05368	Crossbar and armrest support kit		2

Height adjustable armrest n°3

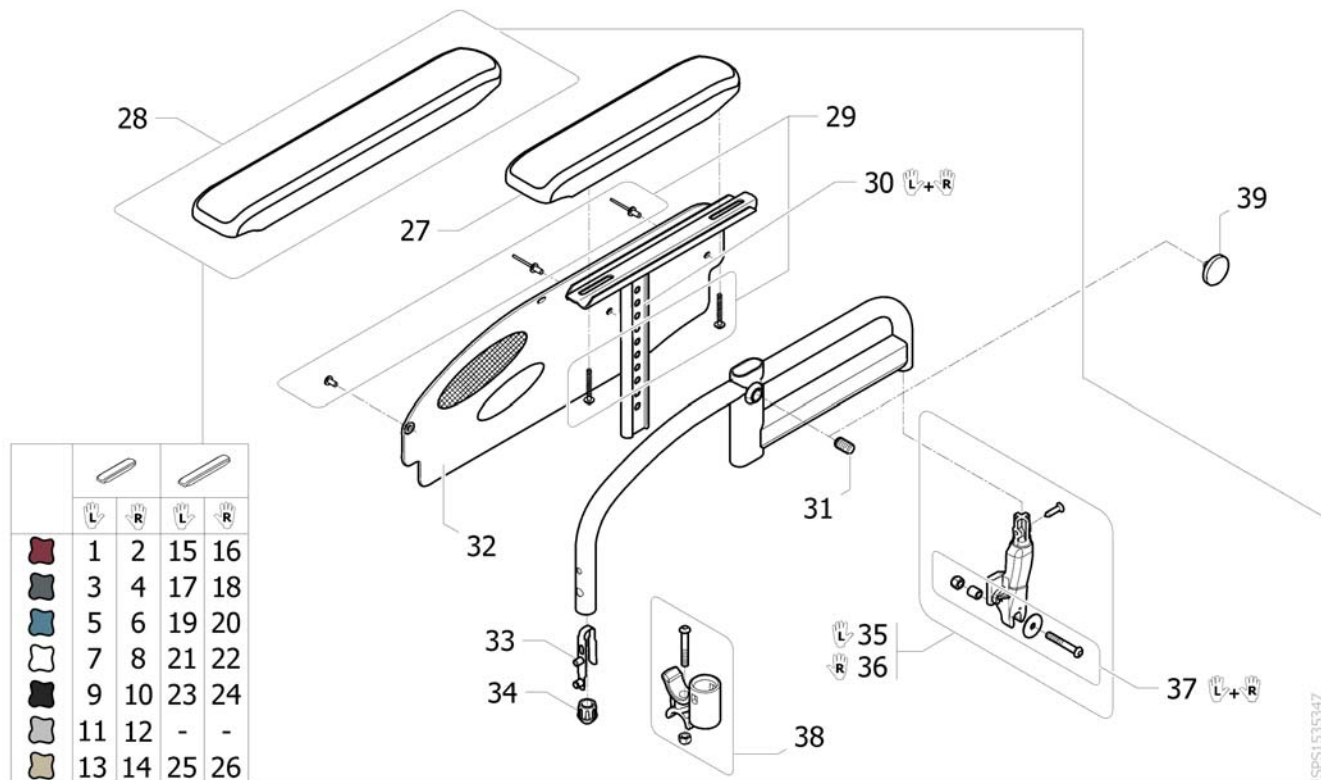


Pos.	Part number	Description	Valid from → until	Qty
1	1428613-0607	Lh Desk Armrest N°3 H & D Adjustable; Dark Red [CE_0780][CK_0599]		1
2	1428612-0607	Rh Desk Armrest N°3 H & D Adjustable; Dark Red [CE_0680][CK_0539]		1
3	1428613-0615	Lh Desk Armrest N°3 H & D Adjustable; Slate Grey [CE_0780][CK_0599]		1
4	1428612-0615	Rh Desk Armrest N°3 H & D Adjustable; Slate Grey [CE_0680][CK_0539]		1
5	1428613-0617	Lh Desk Armrest N°3 H & D Adjustable; Ocean Blue [CE_0780][CK_0599]		1
6	1428612-0617	Rh Desk Armrest N°3 H & D Adjustable; Ocean Blue [CE_0680][CK_0539]		1
7	1428613-0637	Lh Desk Armrest N°3 H & D Adjustable; Pearl White [CE_0780][CK_0599]		1
8	1428612-0637	Rh Desk Armrest N°3 H & D Adjustable; Pearl White [CE_0680][CK_0539]		1
9	1428613-0609	Lh Desk Armrest N°3 H & D Adjustable; Anthracite Black [CE_0780][CK_0599]		1
10	1428612-0609	Rh Desk Armrest N°3 H & D Adjustable; Anthracite Black [CE_0680][CK_0539]		1
11	1428613-0515	Lh Desk Armrest N°3 H & D Adjustable; Pearl Grey [CE_0780][CK_0599]		1
12	1428612-0515	Rh Desk Armrest N°3 H & D Adjustable; Pearl Grey [CE_0680][CK_0539]		1
13	1428613-0616	Lh Desk Armrest N°3 H & D Adjustable; Ice Brown [CE_0780][CK_0599]	→ 2011-12-31	1
14	1428612-0616	Rh Desk Armrest N°3 H & D Adjustable; Ice Brown [CE_0680][CK_0539]	→ 2011-12-31	1
15	1428615-0607	Lh Long Armrest N°3 H & D Adjustable; Dark Red [CE_0790][CK_0600]		1
16	1428614-0607	Rh Long Armrest N°3 H & D Adjustable; Dark Red [CE_0690][CK_0540]		1
17	1428615-0615	Lh Long Armrest N°3 H & D Adjustable; Slate Grey [CE_0790][CK_0600]		1
18	1428614-0615	Rh Long Armrest N°3 H & D Adjustable; Slate Grey [CE_0690][CK_0540]		1

Height adjustable armrest n°3

Pos.	Part number	Description	Valid from → until	Qty
19	1428615-0617	Lh Long Armrest N°3 H & D Adjustable; Ocean Blue <i>[CE_0790][CK_0600]</i>		1
20	1428614-0617	Rh Long Armrest N°3 H & D Adjustable; Ocean Blue <i>[CE_0690][CK_0540]</i>		1
21	1428615-0637	Lh Long Armrest N°3; H & D Adjustable; Pearl White <i>[CE_0790][CK_0600]</i>		1
22	1428614-0637	Rh Long Armrest N°3 H & D Adjustable; Pearl White <i>[CE_0690][CK_0540]</i>		1
23	1428615-0609	Lh Long Armrest N°3 H & D Adjustable; Anthracite Black <i>[CE_0790][CK_0600]</i>		1
24	1428614-0609	Rh Long Armrest N°3 H & D Adjustable; Anthracite Black <i>[CE_0690][CK_0540]</i>		1
25	1428615-0616	Lh Long Armrest N°3 H & D Adjustable; Ice Brown <i>[CE_0790][CK_0600]</i>	→ 2011-12-31	1
26	1428614-0616	Rh Long Armrest N°3 H & D Adjustable; Ice Brown <i>[CE_0690][CK_0540]</i>	→ 2011-12-31	1
27	1434676	Short armpad		2
28	1434677	Long Armpad Flat Support		2
29	1535946	Armrest N°3 Screws Kit		2
30	1524058	Armpad Support Pair		1
31	5106533	Locking screw M12X10		2
32	1426725	Skirtguard black for armrest 3 <i>Replace 1532801 -1532802</i>		2
33	1018225	Double stud spring		2
34	0572-03405	Short conical cap		2
35	1431284	Lh Armrest Rear Support Kit		1
36	1430423	Rh Armrest Rear Support Kit		1
37	1573039	Fixed armrest N°2/3 fixation kit (pair)		1
38	0572-05368	Crossbar and armrest support kit		2
39	1545574	Flat knurled screw M12 X 1 armrest n°3		2


Height adjustable armrest n°4

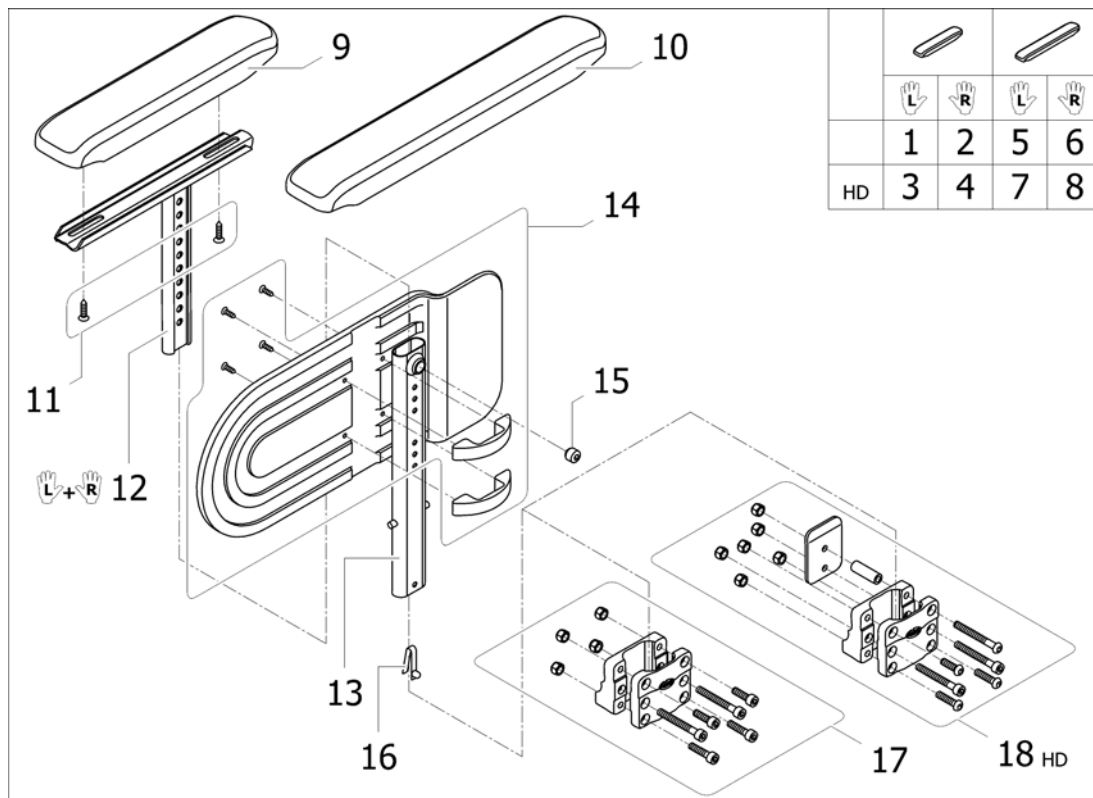

ISPS14963381

Pos.	Part number	Description	Valid from → until	Qty
1	1449988	Adjustable Armrest N°4, Short Pad Lh [CE_0795][CK_0593]		1
2	1449987	Adjustable Armrest N°4, Short Pad Rh [CE_0695][CK_0533]		1
3	1487438	Adjustable Armrest N°4, Short Pad Lh Hd [HD+CE_0795][HD+CK_0593]		1
4	1487437	Adjustable Armrest N°4, Short Pad Rh Hd [HD+CE_0695][HD+CK_0533]		1
5	1450016	Adjustable Armrest N°4, Long Pad Lh [CE_0796][CK_0596]		1
6	1450015	Adjustable Armrest N°4, Long Pad Rh [CE_0696][CK_0536]		1
7	1487440	Adjustable Armrest N°4, Long Pad Lh Hd [HD+CE_0796][HD+CK_0596]		1
8	1487439	Adjustable Armrest N°4, Long Pad Rh Hd [HD+CE_0696][HD+CK_0536]		1
9	1434676	Short armpad		2
10	1434677	Long Armpad Flat Support		2
11	0029-40532	Fbz Screw D 4,8X19 (Qty/pack:20)		4
12	1524058	Armpad Support Pair		1
13	1443058	Lowest Arm Rest Tube Support N°4		2
14	1430425	Skirtguard Armrest N°4 Fixation Kit		2
15	5106533	Locking screw M12X10		2
16	0027-10003	Spring pin locked armrest		2
17	1430421	Central armrest support		2
18	1532575	Central Support Armrest 1 Kit HD		1


Hemi Dartex armpad n°4 multiposition standard

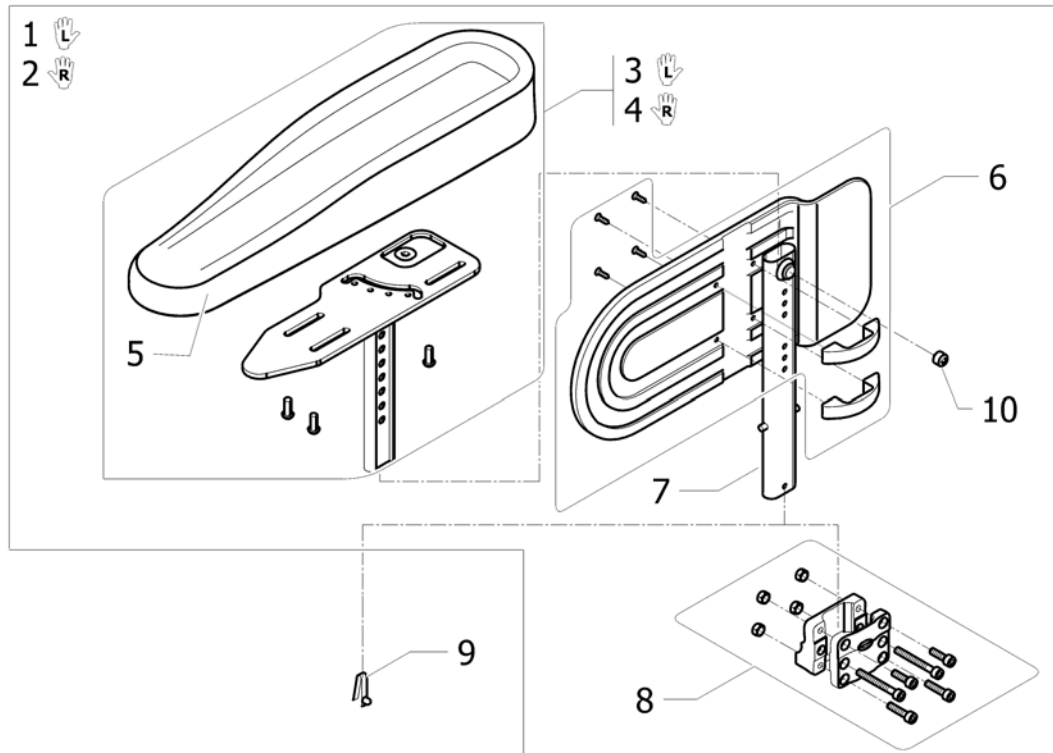

ISPS1548442

Pos.	Part number	Description	Valid from → until	Qty
1	1496103	Lh hemi armrest N°4 multiposition [CK_0615]		1
2	1496091	Rh hemi armrest N°4 multiposition [CK_0555]		1
3	1493220	Lh hemi dartex armpad complete		1
4	1493219	Rh hemi dartex armpad complete		1
5	1493251	Hemi dartex armpad		2
6	1430425	Skirtguard Armrest N°4 Fixation Kit		2
7	1443058	Lowest Arm Rest Tube Support N°4		2
8	1430421	Central armrest support		2
9	0027-10003	Spring pin locked armrest		2
10	5106533	Locking screw M12X10		2




Mudguard Xtra

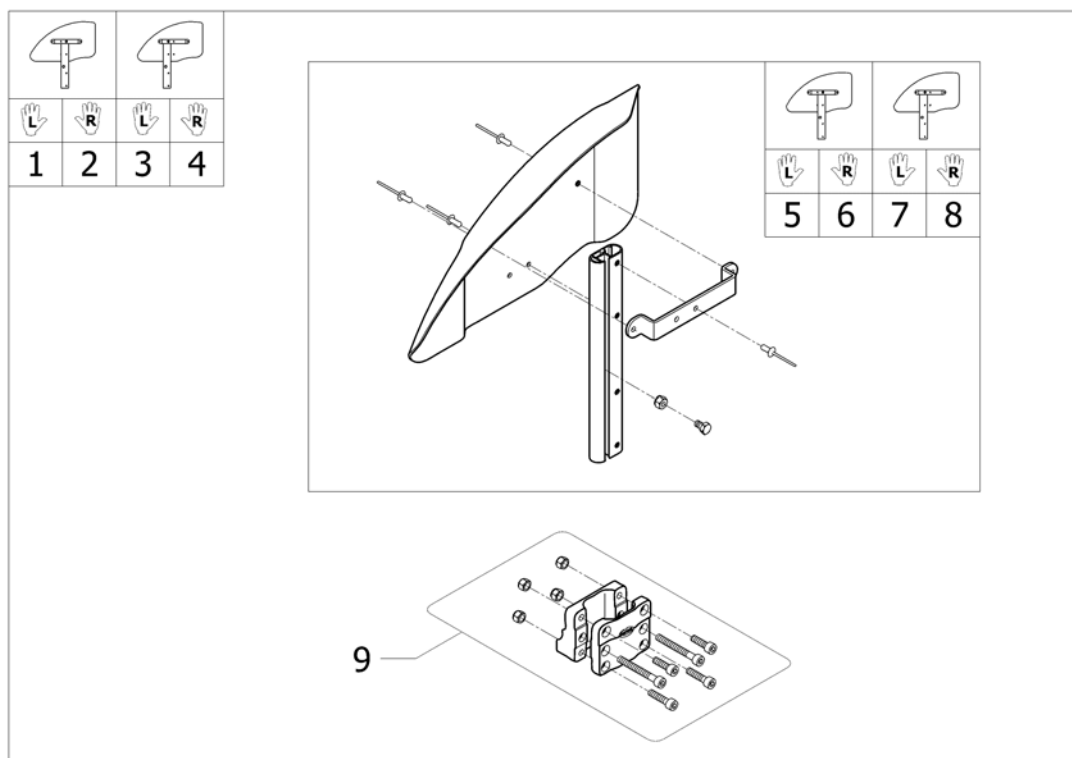

ISPS1495618 ~ sps1514255_ji

Pos.	Part number	Description	Valid from → until	Qty
1	1486042	Lh Mudguard A3 Complete [CE_1190][CK_1055/CK_1060][CN_0160/CN_0162/CN_0170/CN_0171/CN_0250/CN_0255]		1
2	1486041	Rh Mudguard A3 Complete [CE_1190][CK_1055/CK_1060][CN_0160/CN_0162/CN_0170/CN_0171/CN_0250/CN_0255]		1
3	1492739	Lh Mudguard A3-A3J Complete Back Position [CE_1200/CE_1201][CK_1070/CK_1075][CN_0161/CN_0240/CN_0251]		1
4	1492738	Rh Mudguard A3-A3J Complete Back Position [CE_1200/CE_1201][CK_1070/CK_1075][CN_0161/CN_0240/CN_0251]		1
5	1488806	Lh Mudguard + Fixation		1
6	1488805	Rh Mudguard + Fixation		1
7	1558371	Lh Mudguard back position + fixation		1
8	1558372	Rh Mudguard back position + fixation		1
9	1430421	Central armrest support		2


Front Wheels

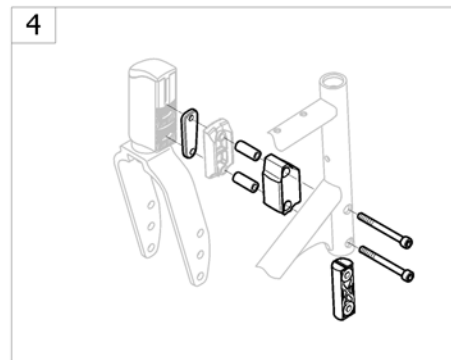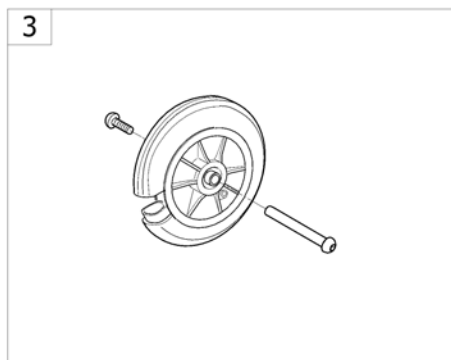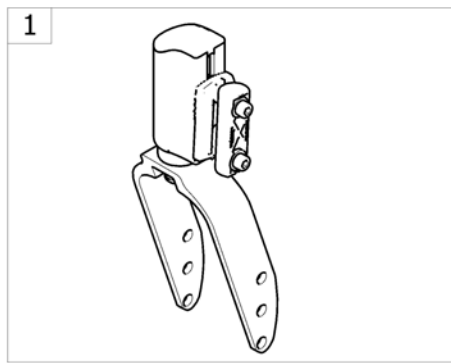


<SPACTION4ING50

Pos.	Part number	Description	Valid from → until	Qty
1	-	Forks		1
2	-	Forks supports adjustable		1
3	-	Front wheels <i>Replace!SPS1496344 ISPS1496369</i>		1
4	-	Front wheel extension kit		1

Forks

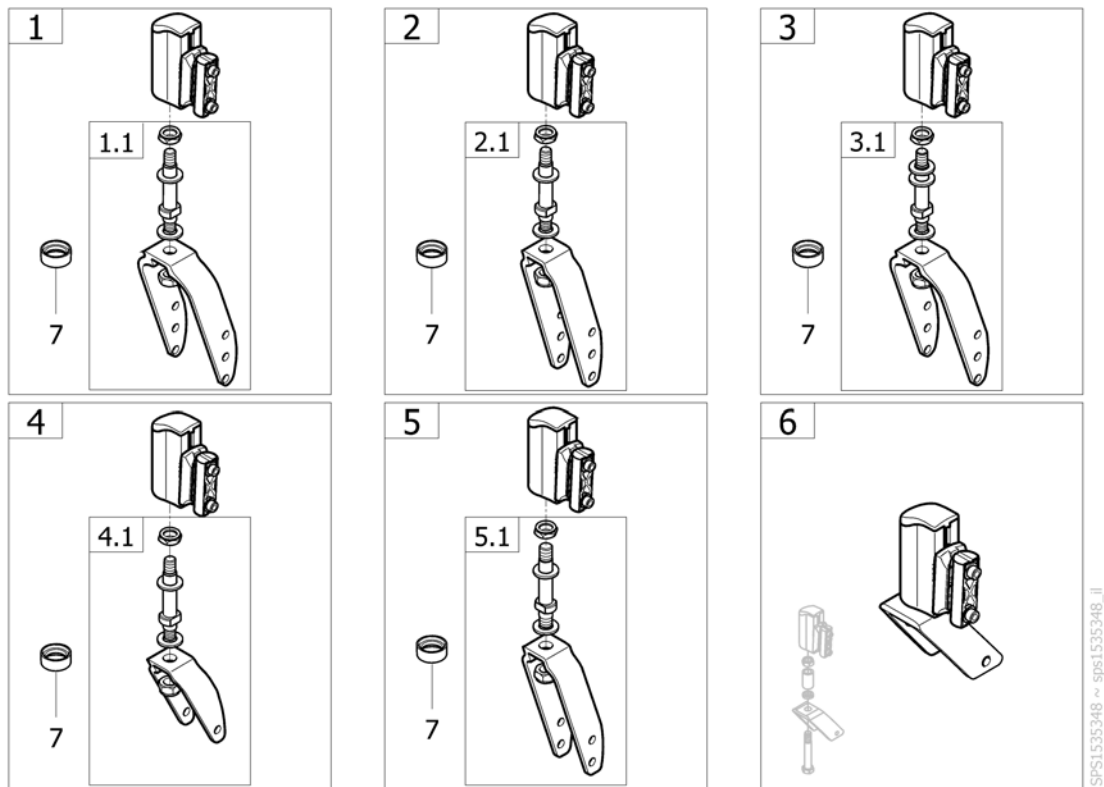


Pos.	Part number	Description	Valid from → until	Qty
1	1450025	Kit fork Action alu black W61, 3 positions + axle + support adj		2
1.1	1444802	Kit fork Action alu black W61, 3 positions + axle		2
2	1450026	Kit fork Action alu black W36 3 positions + axle + support adj [A3NGXtra][CK_0890/CK_0900]		2
2.1	1444803	Kit fork Action alu black W36, 3 positions + axle [A3NGXtra][CK_0890/CK_0900]		2
3	1538177	Kit fork Action alu black W61, 3 positions + axle + support adj + spacers (wheel 5" 1/2) [CE_1052][CK_0901]		2
3.1	1515699	Kit fork Action alu black W61, 3 positions + axle (wheel 5" 1/2) [CE_1052][CK_0901]		2
4	1450027	Kit fork Action alu black W36, 1 position + axle + support adj [CE_0310][CK_0920]		2
4.1	1451775	Kit fork Action alu black W36, 1 position + axle [CE_0310][CK_0920]		2
5	1513247	Kit fork Action alu black W36, 2 positions + axle + support adj [A3NGCompact/CE_1051] - Replace 1529518		2
5.1	1513536	Kit fork Action alu black W36, 2 positions + axle [A3NGCompact/CE_1051]		2
6	1548510	Kit fork Action alu black W40, 1 position + axle + support adj ultra low [CE_0315][CK_0265]		2
7	1529516-0618	Fork ring Champagne		2

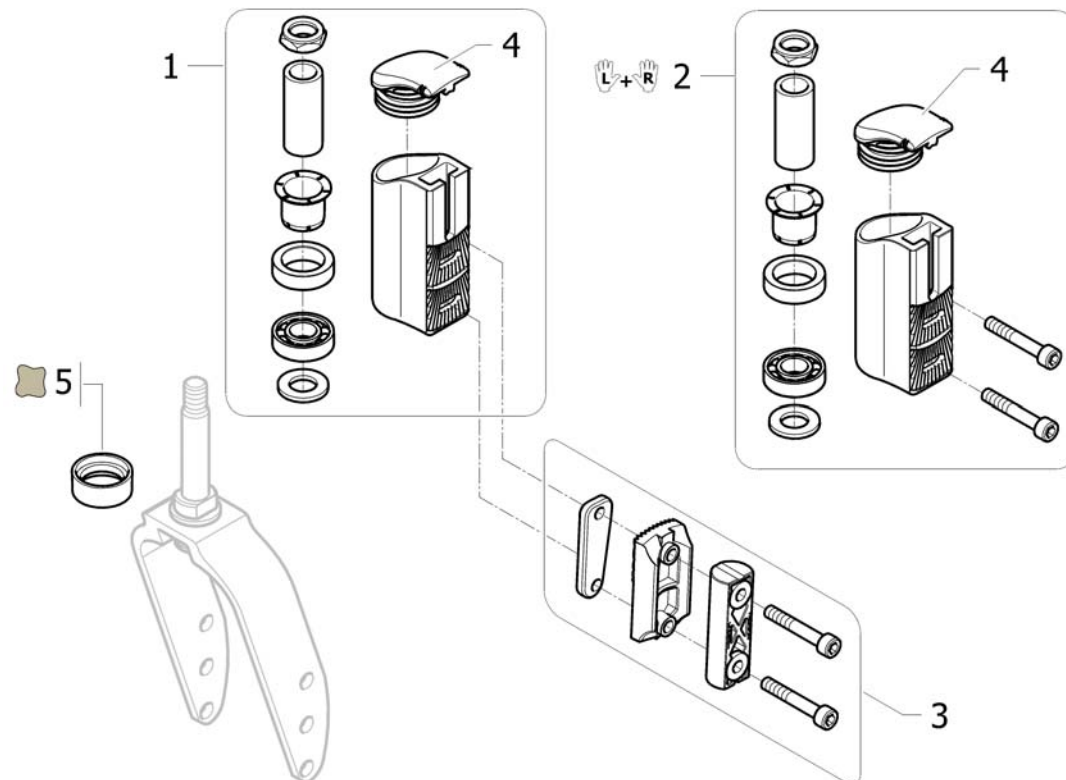Forks supports adjustable


ISPS1535968 ~ sps1535348_11

Pos.	Part number	Description	Valid from → until	Qty
1	1444804	Kit fork support adj Action alu black, bearing housing only (2011 version)		2
2	1556557	Kit fork support adj Action alu black, bearing housing + screws (pair, Transformation kit to 2011 version)	→ 2011-11-13	1
3	1450028	Kit fork support adj Action alu black, plate & inserts only		2
4	1540575	Plastic plug, castor housing		2
5	1529516-0618	Fork ring Champagne		2

Front wheels

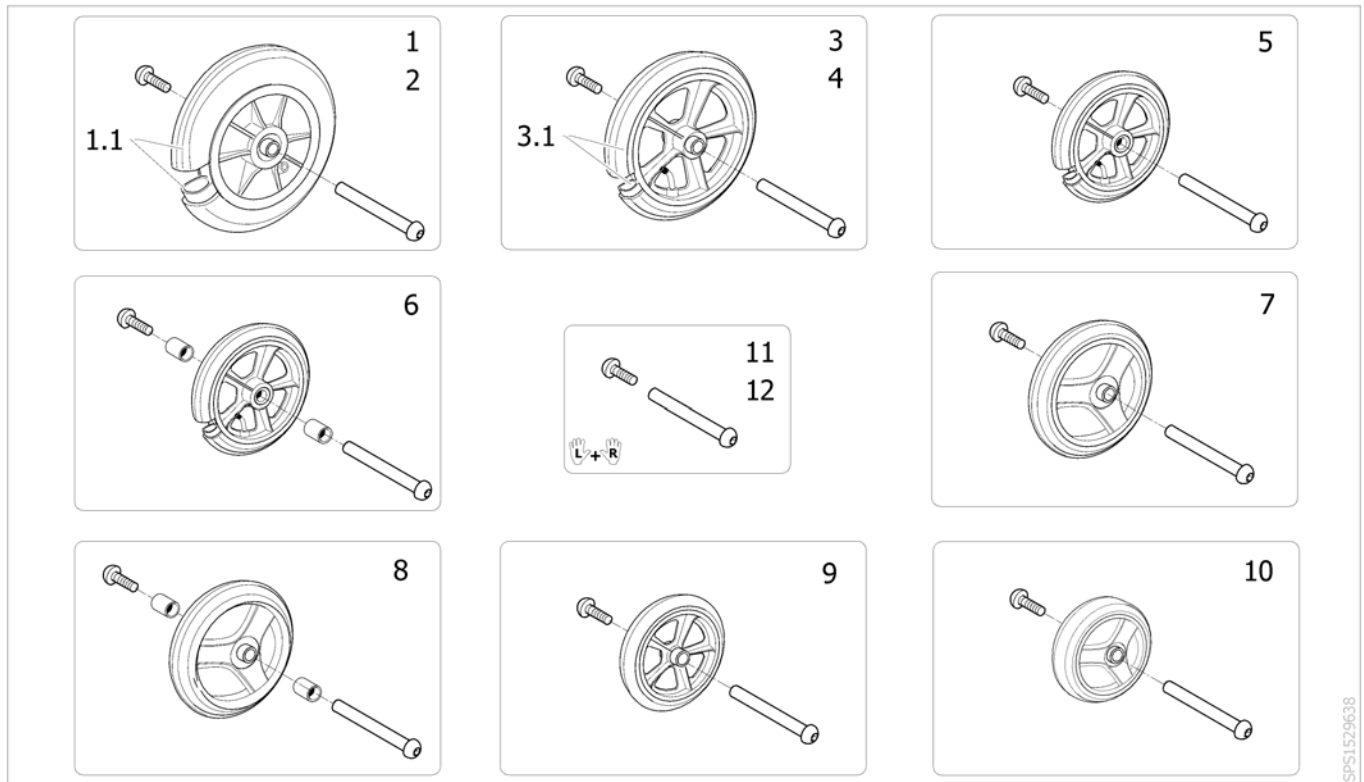


Pos.	Part number	Description	Valid from → until	Qty
1	0572-02071	Front wheel 8" x 2" (200x50) + axle, Pneumatic tyre, L60 mm		2
1.1	1430874	Pneumatic tyre 8" x 2" + inner tube		2
2	SP1584031	Front wheel 8" x 2" (200x45) + axle, Solid tyre, L60 mm <i>Replace 0572-06128 for wheelchair after 2014/05/15</i>	2014-05-16 →	2
2	SP1581577	Pair front wheel 8" x 2" (200x45) + axle, Solid tyre, L60 mm <i>Replace 0572-06128 for wheelchair before 2014/05/16</i>	→ 2014-05-15	1
3	0572-01859	Front wheel 8" x 1.1/4" + axle, Pneumatic tyre, L60 mm		2
3.1	1430873	8" x 1.1/4" pneumatic tyre + inner tube		2
4	SP1584032	Front castor 8" x 1.1/4" (200x30) + axle, Solid tyre, L60 mm <i>Replace 0572-02013 for wheelchair after 2014/06/02</i>	2014-06-03 →	2
4	SP1582956	Pair front wheel 8" x 1.1/4" (200x30) + axle, solid tyre, L60 mm <i>Replace 0572-02013 for wheelchair before 2014/06/03</i>	→ 2014-06-02	1
5	SP1589882	Front wheel 6"x 1.1/4" + axle, Solid type, L36 mm <i>Replace 1450174 for wheelchair after 2015/03/31</i>	2015-04-01 →	2
5	SP1589883	Pair front wheel 6"x 1.1/4" + axle, Solid type, L36 mm <i>Replace 1450174 for wheelchair before 2015/04/01</i>	→ 2015-03-31	1
6	SP1589884	Front wheel 6" x 1.1/4" + axle, Solid tyre, L60 mm <i>Replace 1428895 for wheelchair after 2015/03/31</i>	2015-04-01 →	2
6	SP1589885	Pair front wheel 6" x 1.1/4" + axle, Solid tyre, L60 mm <i>Replace 1428895 for wheelchair before 2015/04/01</i>	→ 2015-03-31	1
7	1450622	A4 6 Castor Medium Hard Type Complete L :36 mm		2
8	1450176	Front wheel 6" x 1.1/4" + axle, Medium hard, L60 mm		2
9	1515254	Front wheel 5,5"x 1.3/4" + axle, Medium Hard, Solid tyre, L60 mm	2012-10-30 →	2
9	1567974	Pair front wheel 5,5"x 1.3/4" + axle, Medium Hard, Solid tyre, L60 mm	2004-11-25 → 2012-10-29	1
10	SP1585997	Front wheel 125x30 solid tyre + axle for fork width 36mm <i>Replace 1450173 for wheelchair after 27/01/2015</i>	2015-01-28 →	2
10	SP1586003	Pair front wheels 125x30 solid tyre + axle for fork width 36mm <i>Replace 1450173 for wheelchair before 28/01/2015</i>	→ 2015-01-27	1
11	1561355	Kit axle M8 for forks L60 (Pair)		1
12	1561356	Kit axle M8 for forks L36 (Pair)		1

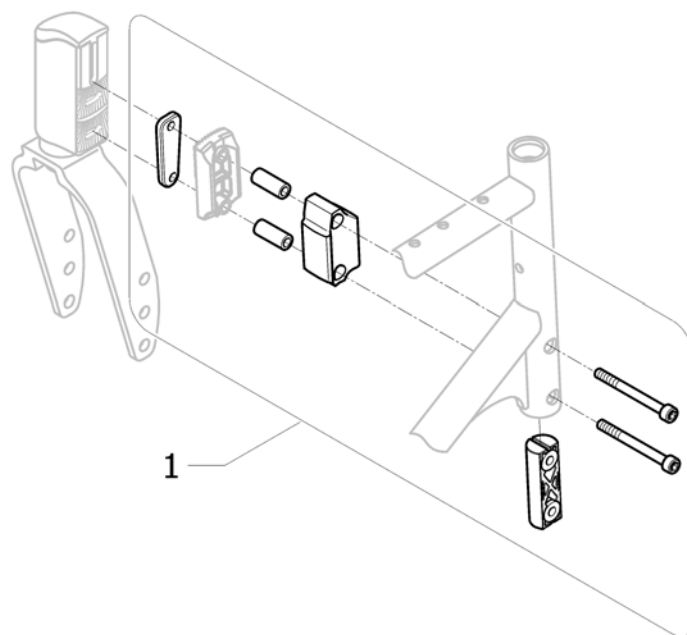

Pos.	Part number	Description	Valid from → until	Qty
1	1497767	Front wheel extension kit		1





Rear Wheels

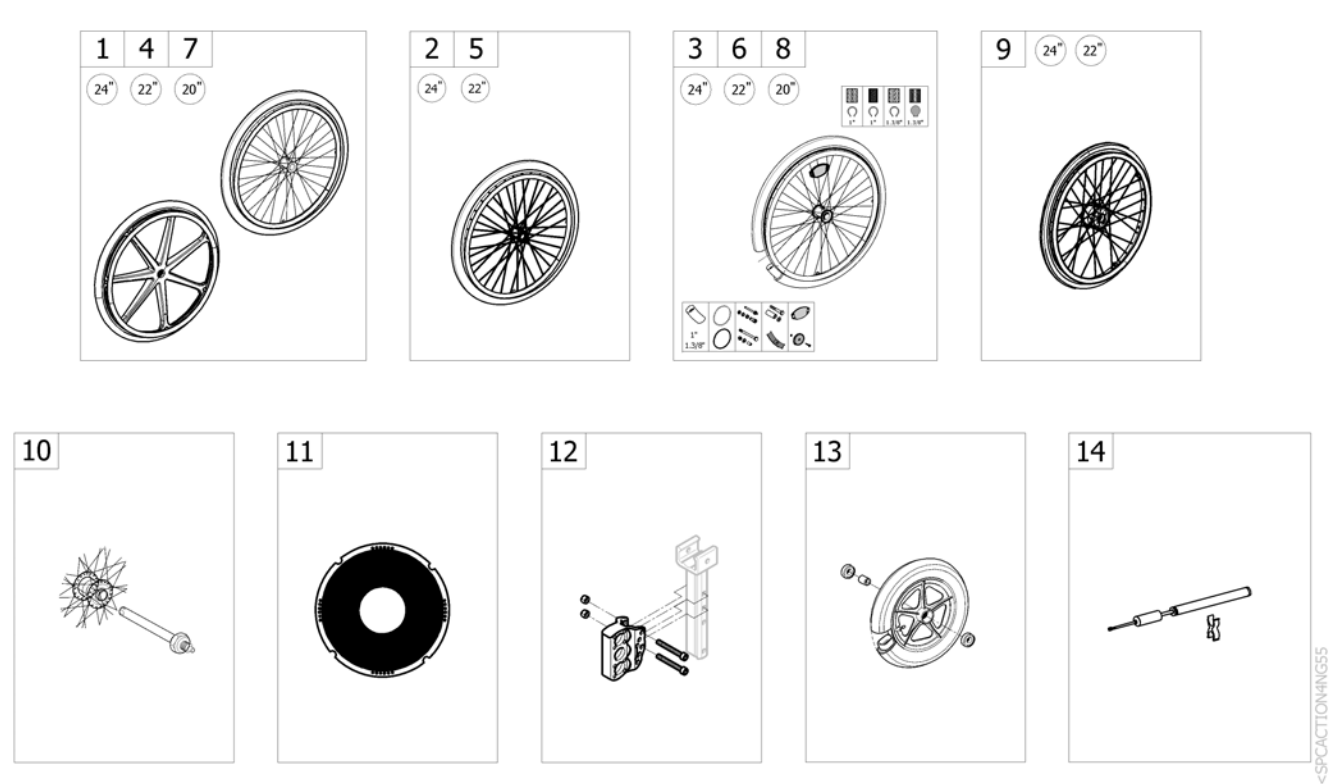

Pos.	Part number	Description	Valid from → until	Qty
1	-	24" Standard Wheels		1
2	-	24" Heavy Duty Wheels		1
3	-	24" Standard Wheels accessories		1
4	-	22" Standard Wheels		1
5	-	22" Heavy Duty Wheels		1
6	-	22" Standard Wheels accessories		1
7	-	20" Standard Wheels		1
8	-	20" Standard Wheels accessories		1
9	-	24" / 22" Hub brake wheels		1
10	-	Quick release axle dia 12 Action		1
11	-	Spoke Guard		1
12	-	Rear Wheel Supports		1
13	-	12" Rear Wheels		1
14	-	Pump		1

24" Standard Wheels

24"

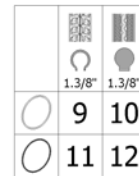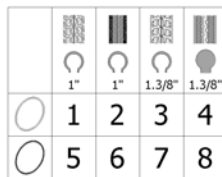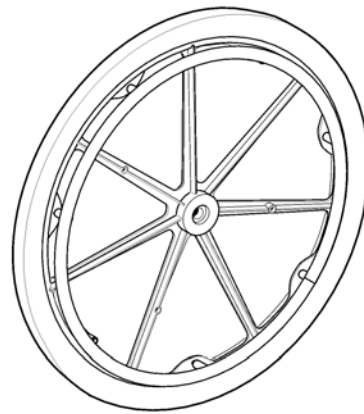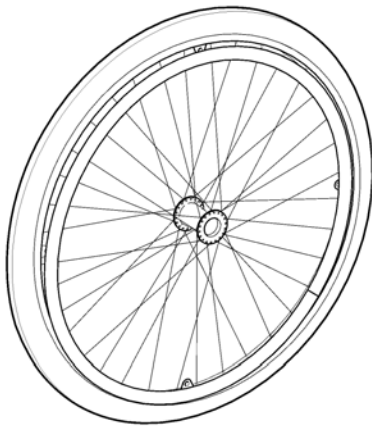


ISPS1535349

Pos.	Part number	Description	Valid from → until	Qty
1	1498425	24" x 1" wheel,spoked,anodized handrim,notched pneumatic <i>Replace 1555608</i>		2
2	1498431	24" x 1" wheel,spoked,anodized handrim,non marking pneumatic <i>Replace 1555618</i>		2
2	1498431-0618	24" x 1" wheel, spoked champagne, anodized handrim, non marking pneumatic		2
3	1498423	24" x 1.3/8" wheel,spoked,anodized handrim,pneumatic		2
4	1498263	24" x 1.3/8" wheel,spoked,anodized handrim,solid tyre,		2
5	1498426	24" x 1" wheel,spoked,anti-slip handrim,notched pneumatic <i>Replace 1555706</i>		2
6	1534211-0618	24" x 1" wheel,spoked,champagne,anti-slip handrim,non marking		2
7	1498424	24" x 1.3/8" wheel,spoked,anti-slip handrim,pneumatic <i>Replace 1555624</i>		2
8	1498422	24" x 1.3/8" wheel,spoked,anti-slip handrim,solid tyre <i>Replace 1555623</i>		2
9	1572540	24" x 1.3/8" composite, anodized handrim, pneumatic	2013-10-14 →	2
9	1423312	Pair of wheels 24" x 1.3/8"composite, anodized handrim, pneumatic <i>Replace 0572-05909</i>	2009-09-16 → 2013-10-13	1
10	1572541	24" x 1.3/8" composite, anodized handrim, solid tyre	2013-10-14 →	2
10	1423310	Pair of wheels 24" x 1.3/8"composite, anodized handrim, solid tyre <i>Replace 0572-05908</i>	2009-09-16 → 2013-10-13	1
11	1572542	24" x 1.3/8" composite, anti-slip handrim, pneumatic	2013-10-14 →	2
11	1419833	Pair of wheels 24"x1.3/8"composite, anti-slip handrim, pneumatic <i>Replace 0572-05193</i>	2009-09-16 → 2013-10-13	1
12	1572543	24" x 1.3/8" composite, anti-slip handrim, solid tyre	2013-10-14 →	2
12	1419835	Pair of wheels 24"x1.3/8"composite, anti-slip handrim, solid tyre <i>Replace 0572-05195</i>	2009-09-16 → 2013-10-13	1

24" Heavy Duty Wheels

24"

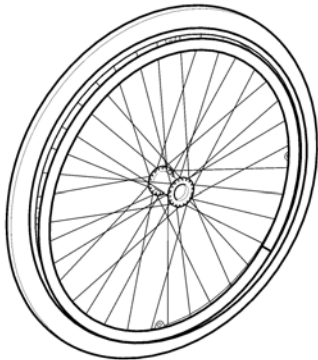


		
		
	1.3/8"	1.3/8"
	1	2
	3	4

19713331

Pos.	Part number	Description	Valid from → until	Qty
1	1498486	Rear wheel, 24", HD, pneumatic tyre 1.3/8", aluminium handrim		2
2	1498467	Rear wheel, 24", HD, solid tyre 1.3/8", aluminium handrim		2
3	1498492	Rear wheel, 24", HD, pneumatic tyre 1.3/8", anti-slip handrim		2
4	1498478	Rear wheel, 24", HD, solid tyre 1.3/8", anti-slip handrim		2

24" Standard Wheels accessories

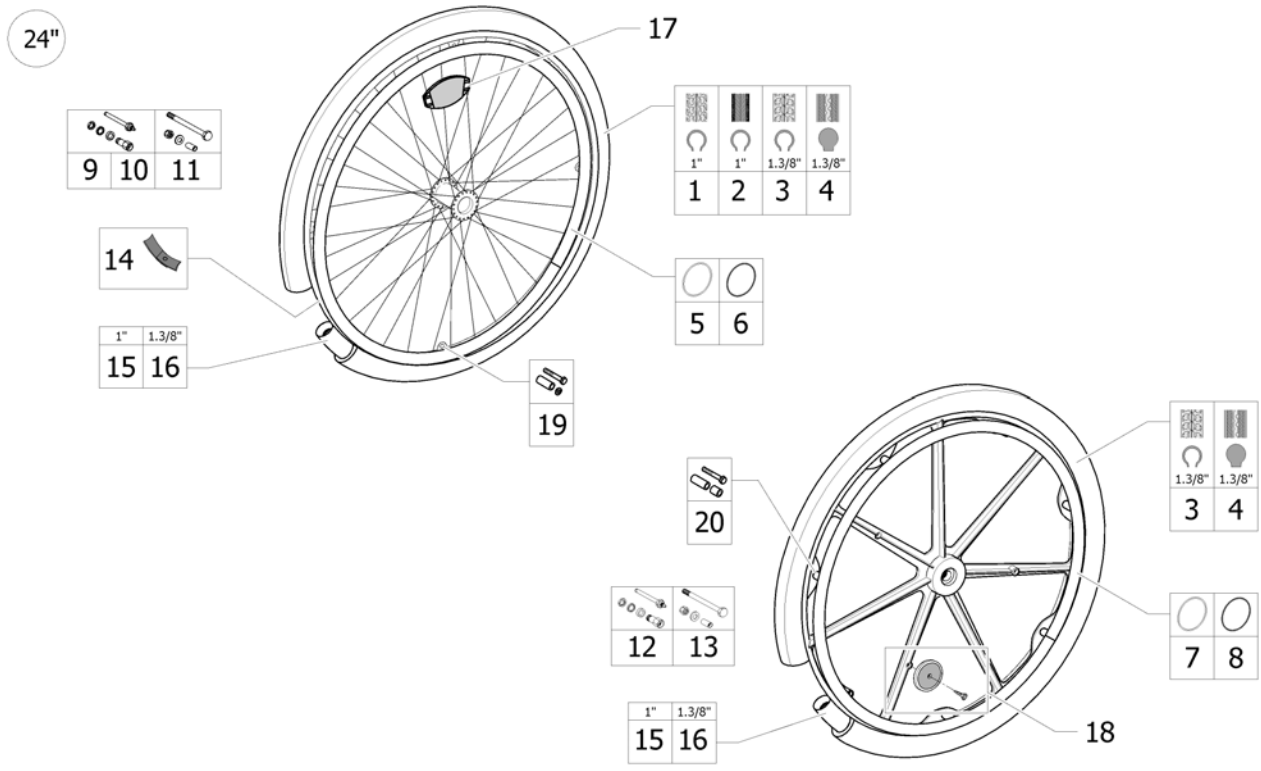


ISPS1535350

Pos.	Part number	Description	Valid from → until	Qty
1	0036-30013	24" x 1. notched pneumatic tyre		2
2	1026138	24" x 1. non marking pneumatic tyre		2
3	0036-30001	Pneumatic tyre, 24" x 1.3/8", grey		2
4	0036-10013	Solid tyre, 24" x 1.3/8", grey		2
5	1505291	Kit aluminium handrim 24", 4 fixations	2009-09-16 →	2
6	1505298	Kit anti-slip handrim 24", 4 fixations	2009-09-16 →	2
7	1581659	24" anodized handrim + fixations, 7 points	2014-04-28 →	2
7	1431317	24" anodized handrim + fixations, 7 points	→ 2014-04-27	2
7	1545554	24" anodized handrim + fixations, 7 points (MBL)	2010-09-23 → 2011-06-21	2
8	1581660	24" Anti-slip handrim + fixations, 7 points	2014-04-28 →	2
8	1431318	24" Anti-slip handrim + fixations, 7 points	→ 2014-04-27	2
8	1545555	24" anti-slip handrim + fixations, 7 points (MBL)	2010-09-23 → 2011-06-21	2
9	1505288	QR axle with axle bearing Ø12		2
10	1505290	QR axle for hub brake spoked wheels		2
11	1505289	Fixed axle for spoked wheels		2
12	1434129	QR axle for composite wheels		2
13	1431279	Fixed axle for composite wheels		2
14	0036-40003	Rim tape 24"		2
15	1032240	Inner tube, 24" x 1"		2
16	0036-20001	Inner tube, 24" x 1.3/8"		2
17	N90003	Kit reflector for spoked wheel		2
18	9037-30001	Kit reflector for composite wheel		2
19	1573626	Kit handrim fixation for spoked wheel, 4 pts		2
20	1581725	Kit handrim fixation for composite wheel, 7 points	2014-04-28 →	2
20	1573627	Kit handrim fixation for composite wheel, 7 points	→ 2014-04-27	2

22" Standard Wheels

22"


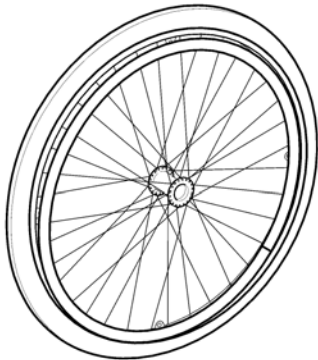


		
		
	1.3/8"	1.3/8"
	1	2
	3	4

19713331

Pos.	Part number	Description	Valid from → until	Qty
1	1498434	22" x 1.3/8" wheel,spoked,anodized handrim,pneumatic		2
2	1498432	22" x 1.3/8" wheel,spoked,anodized handrim,solid tyre,		2
3	1498435	22" x 1.3/8" wheel,spoked,anti-slip handrim,pneumatic		2
4	1498433	22" x 1.3/8" wheel,spoked,anti-slip handrim,solid tyre		2


22"

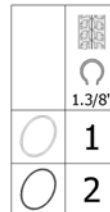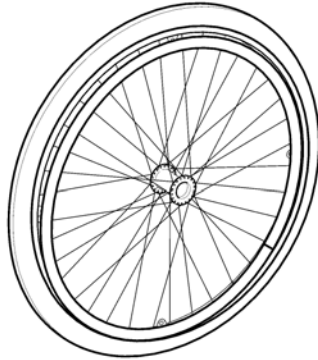

19713331

Pos.	Part number	Description	Valid from → until	Qty
1	1498493	Rear wheel, 22", HD, pneumatic tyre 1.3/8", aluminium handrim		2
2	1498494	Rear wheel, 22", HD, pneumatic tyre 1.3/8", anti-slip handrim		2

22" Standard Wheels accessories

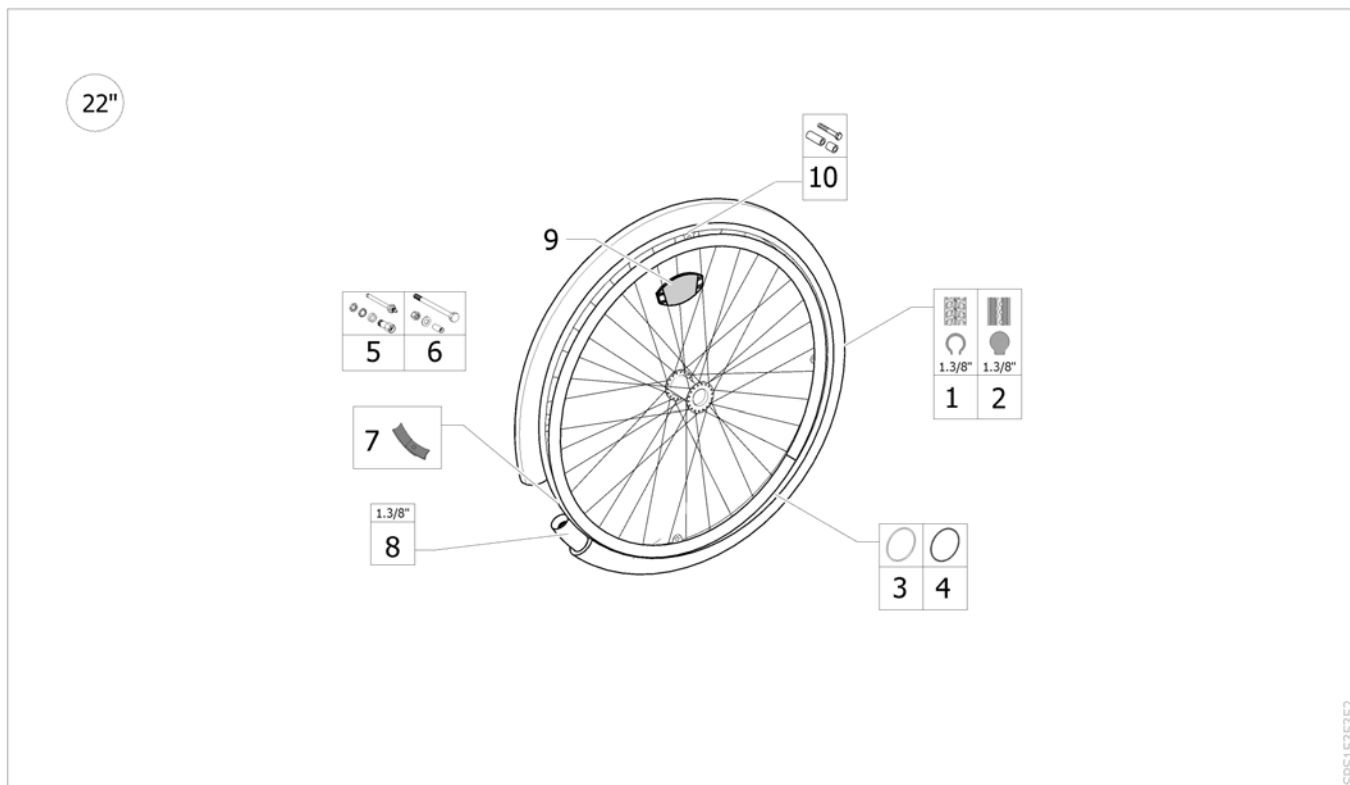


ISPS1535352

Pos.	Part number	Description	Valid from → until	Qty
1	0036-30035	Pneumatic tyre, 22" x 1.3/8", grey		2
2	0036-10014	Solid tyre, 22" x 1.3/8", grey		2
3	1505299	Kit aluminium handrim 22", 4 fixations		2
4	1505920	Kit anti-slip handrim 22", 4 fixations		2
5	1505288	QR axle with axle bearing Ø12		2
6	1505289	Fixed axle for spoked wheels		2
7	5750029	Rim tape 22" <i>Replace 0036-40002</i>		2
8	0036-20007	Inner tube, 22" x 1.3/8"		2
9	N90003	Kit reflector for spoked wheel		2
10	1573626	Kit handrim fixation for spoked wheel, 4 pts		2

20" Standard Wheels

20"

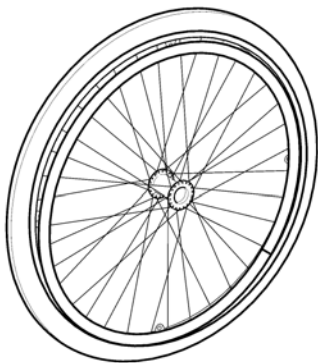


		
		
	1	2
		3

ISPS1524164

Pos.	Part number	Description	Valid from → until	Qty
1	1445041	20" x 1.3/8" wheel,spoked,anodized handrim,pneumatic	→ 2011-12-31	2
2	1445042	20" x 1.3/8" wheel,spoked,anodized handrim,solid tyre,		2
3	1557446	20" x 1.3/8" wheel,spoked,anti-slip handrim,solid tyre		2

20" Standard Wheels accessories

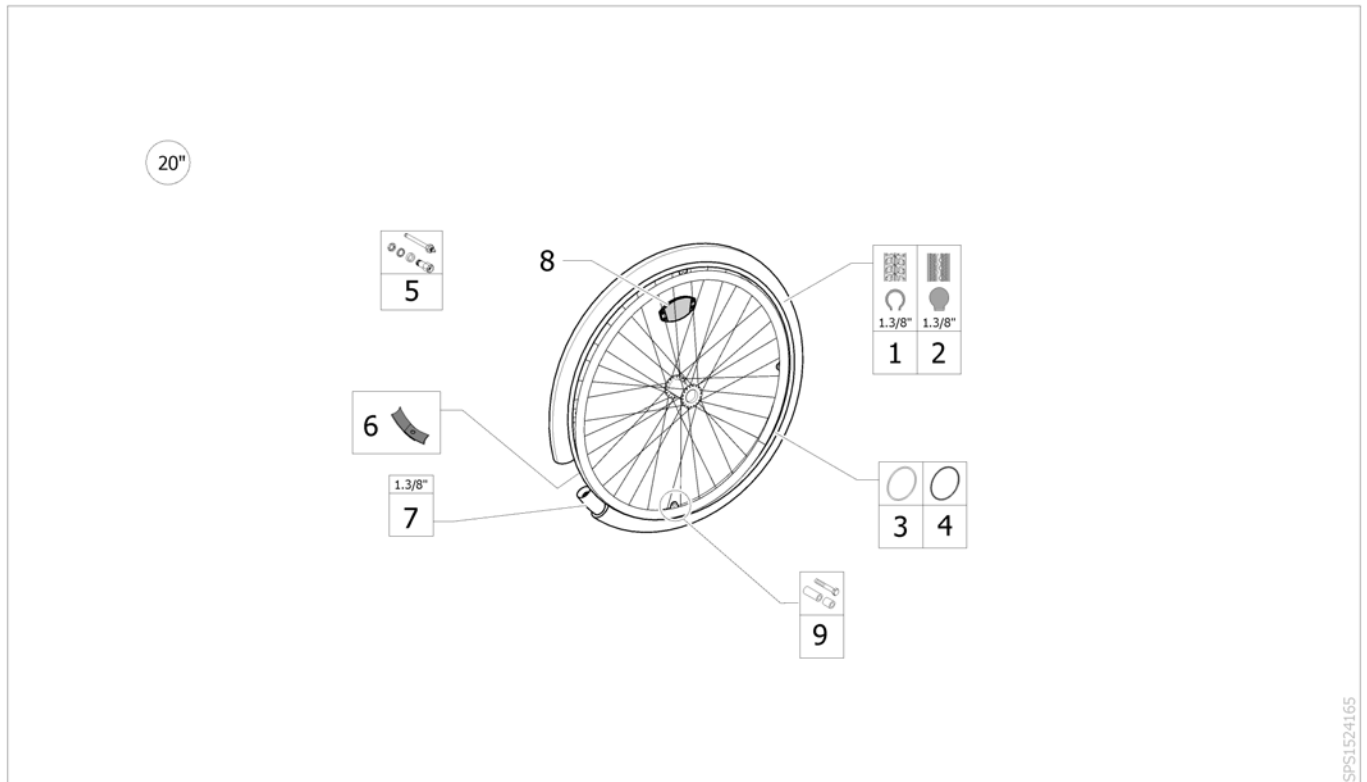


ISPS1524165

Pos.	Part number	Description	Valid from → until	Qty
1	M11009	20" X 1.3/8" Pneumatic	→ 2011-12-31	2
2	1TYR2002	20" x 1/4" solid tyre		2
3	1507241	20" Anodised Handrim		2
4	1557837	20" anti-slip handrim 4 fixations		1
5	1505288	QR axle with axle bearing Ø12		2
6	1507198	20" Rim Tape		2
7	0036-20006	20" Inner Tube		2
8	N90003	Kit reflector for spoked wheel		2
9	1573626	Kit handrim fixation for spoked wheel, 4 pts		2

24" / 22" Hub brake wheels

24" 22"

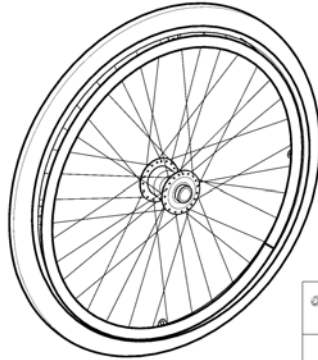


		
	1.3/8"	1.3/8"
24"	1	2
22"	3	4

		
	1.3/8"	1.3/8"
24"	5	6
22"	7	8

ISP1535353

Pos.	Part number	Description	Valid from → until	Qty
1	1498428	Rear wheel, 24", hub brake, pneumatic tyre 1.3/8", aluminium handrim		2
2	1498427	Rear wheel, 24", hub brake, solid tyre 1.3/8", aluminium handrim		2
3	1498466	Rear wheel, 22", hub brake, pneumatic tyre 1.3/8", aluminium handrim		2
4	1498461	Rear wheel, 22", hub brake, solid tyre 1.3/8", aluminium handrim		2
5	1573481	Rear wheel, 24", hub brake, pneumatic tyre 1.3/8", anti-slip handrim		2
6	1573482	Rear wheel, 24", hub brake, solid tyre 1.3/8", anti-slip handrim		2
7	1557823	Rear wheel, 22", hub brake, pneumatic tyre 1.3/8", anti-slip handrim		2
8	1557801	Rear wheel, 22", hub brake, solid tyre 1.3/8", anti-slip handrim		2
9	1505290	QR axle for hub brake spoked wheels		2






Quick release axle dia 12 Action

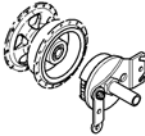						
	=> 03-2007	03-2007 =>	=> 06-2007	06-2007 =>	01-2011 =>	
Action 1NG		3				
Action 2NG		1		4	1	
Action 3	1	1	1	1		1
Action 3NG		1		1		1
Action 3J		1		1		
Action 4	1	1	1	1		1
Action 4NG		1		1		1
Esprit A3 J		1				
Action 2000	1	1				2
Variable +	1	1		1		2

	1	2	3	4
---	---	---	---	---

ISPS1572689


Pos.	Part number	Description	Valid from → until	Qty
1	1498496	Quick release axle Ø12		2
2	0047-10035	Quick release axle Ø12		2
3	1137209	Quick release axle Ø12		2
4	1485144	Quick release axle Ø12		2

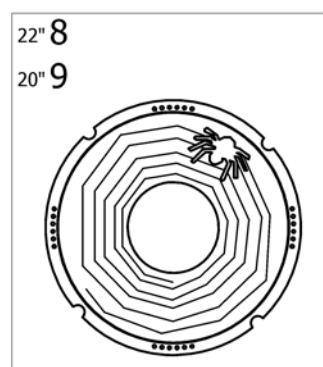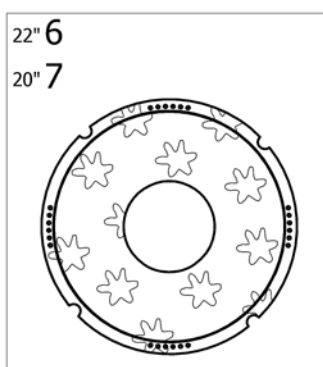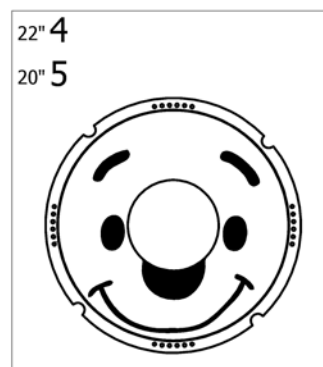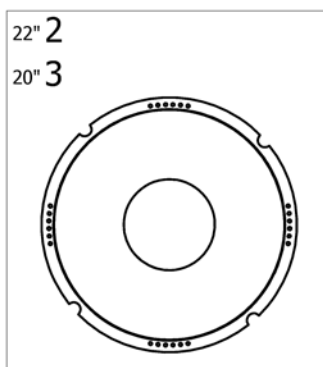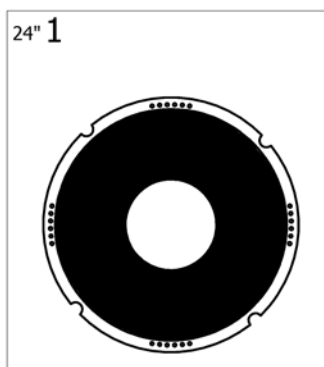Spoke Guard

ISPS1497313 ~ sps1497313_ji





Pos.	Part number	Description	Valid from → until	Qty
1	0789-04400	Spoke guard 24" Black (pair)		1
2	1556990	Spoke guard 22" Transparent (Pair)		1
3	1556989	Spoke guard 20" Transparent (Pair)		1
4	1522862	Spoke Guard 22" Smiley (pair)		1
5	1492138	Spoke Guard 20" Smiley (pair)		1
6	1522864	Spoke Guard 22" Flower (pair)		1
7	1513258	Spoke Guard 20" Flower (pair)		1
8	1522863	Spoke Guard 22" Spider (pair)		1
9	1513257	Spoke Guard 20" Spider (pair)		1

Rear Wheel Supports

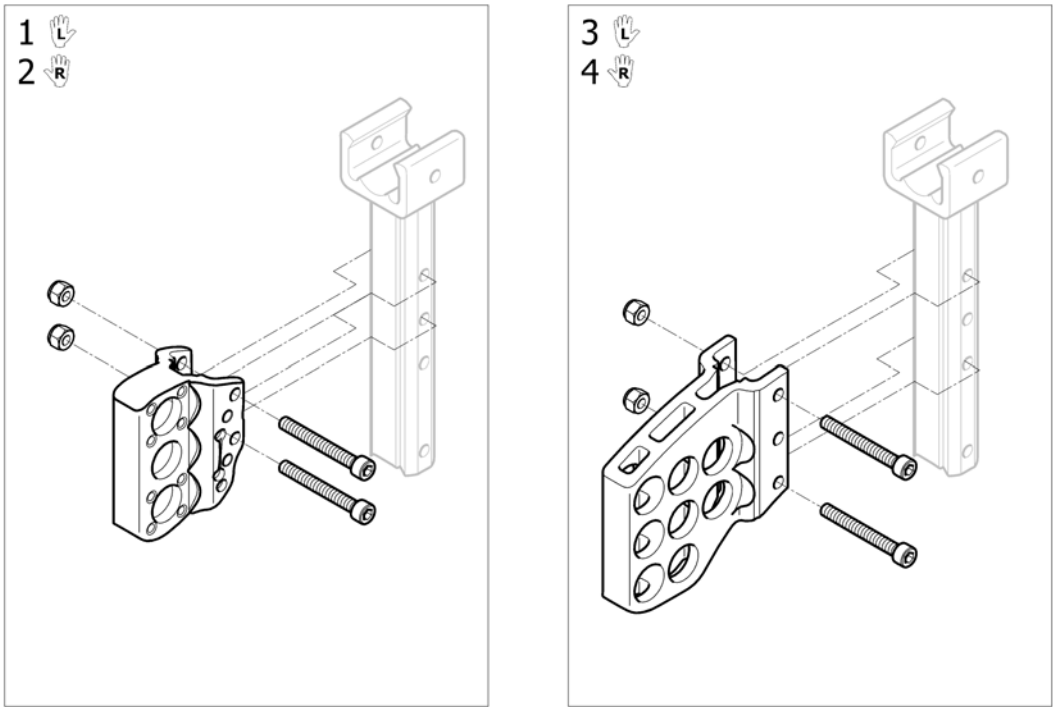

ISPS1535394


Pos.	Part number	Description	Valid from → until	Qty
1	1535060	A4 Standard Rear Wheel Support Lh		1
2	1535059	A4 Standard Rear Wheel Support Rh		1
3	1514945	A4 Active / Amputee rear wheel support LH <i>Replace 1523612</i>		1
4	1514944	A4 Active / Amputee rear wheel support RH <i>Replace 1523612</i>		1

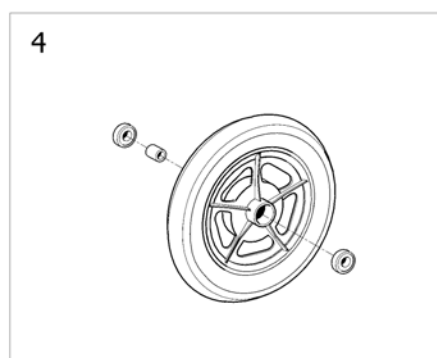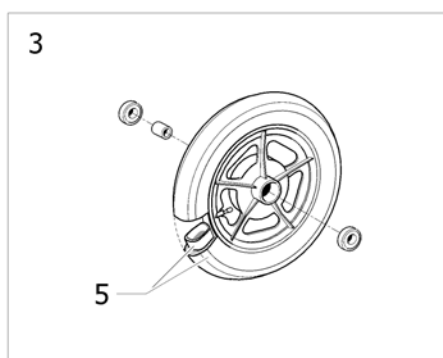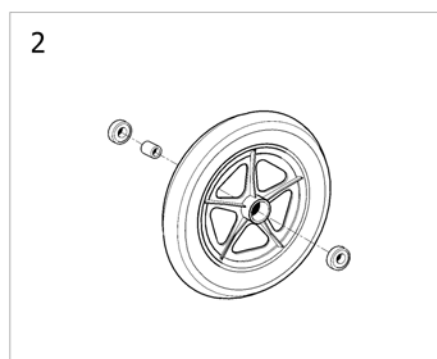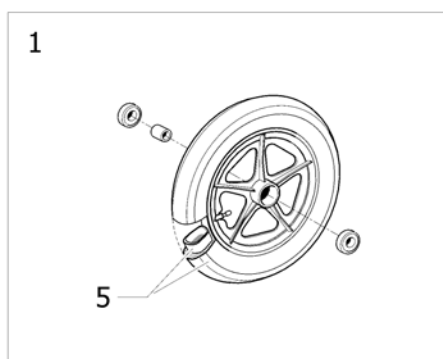12" Rear Wheels

ISPS1535355


Pos.	Part number	Description	Valid from → until	Qty
1	SP1584368	12" Rear wheel transit, pneumatic <i>Replace 6001789</i>	2014-09-17 →	1
1	SP1584050	12" Rear wheel transit, pneumatic (pair)	→ 2014-09-16	1
2	1571005	12" rear wheel, composite, standard hub, solid tyre <i>Replace 1535037</i>	2014-08-18 →	2
2	1530448	12" rear wheel, composite, standard hub, solid tyre (pair)	2004-11-25 → 2014-08-17	1
3	SP1587822	12" rear wheel composite, hub brake pneu (unit) <i>Replace 0572-03002</i>	2015-02-19 →	2
3	SP1587823	12" rear wheel composite, hub brake pneu (pair) <i>Replace 0572-03002</i>	→ 2015-02-18	1
4	1571032	12" rear wheel, composite, hub brake, solid tyre <i>Replace 1557302</i>	2014-09-25 →	2
4	1557447	12" rear wheel, composite, hub brake, solid tyre (pair)	2012-01-01 → 2014-09-24	1
5	1431313	12" transit Pneumatic Tyre + Inner tube		2

Pump

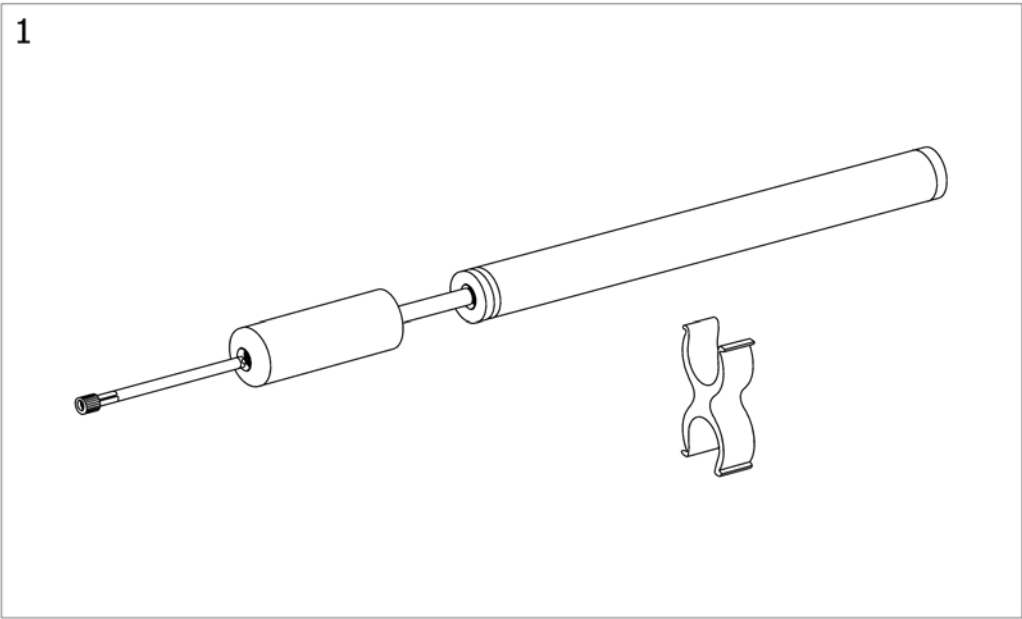

Pos.	Part number	Description	Valid from → until	Qty
1	1419910	Kit pump low pressure + support <i>Replace 1484715</i>		1


Brakes

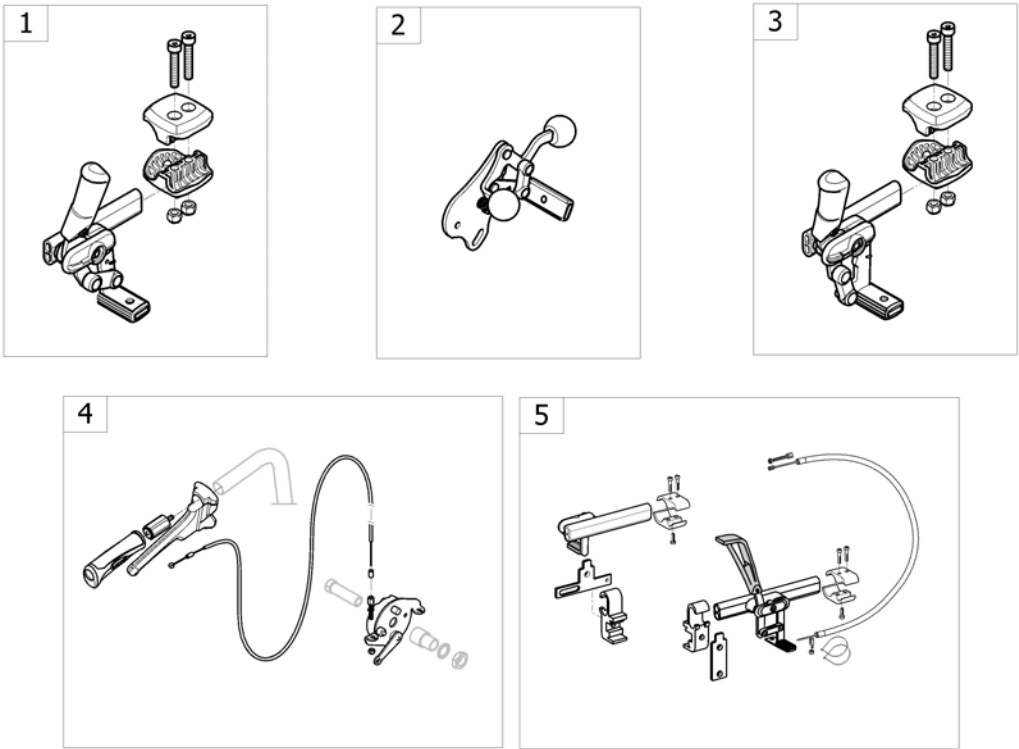

<SPCACTION4NG65


Pos.	Part number	Description	Valid from → until	Qty
1	-	Push Brakes : Standard And With Extension		1
2	-	Push brake Transit		1
3	-	Pull brakes : Standard And With Extension		1
4	-	Hub Brakes		1
5	-	Oad Wire Control Brake		1

Push Brakes : Standard And With Extension

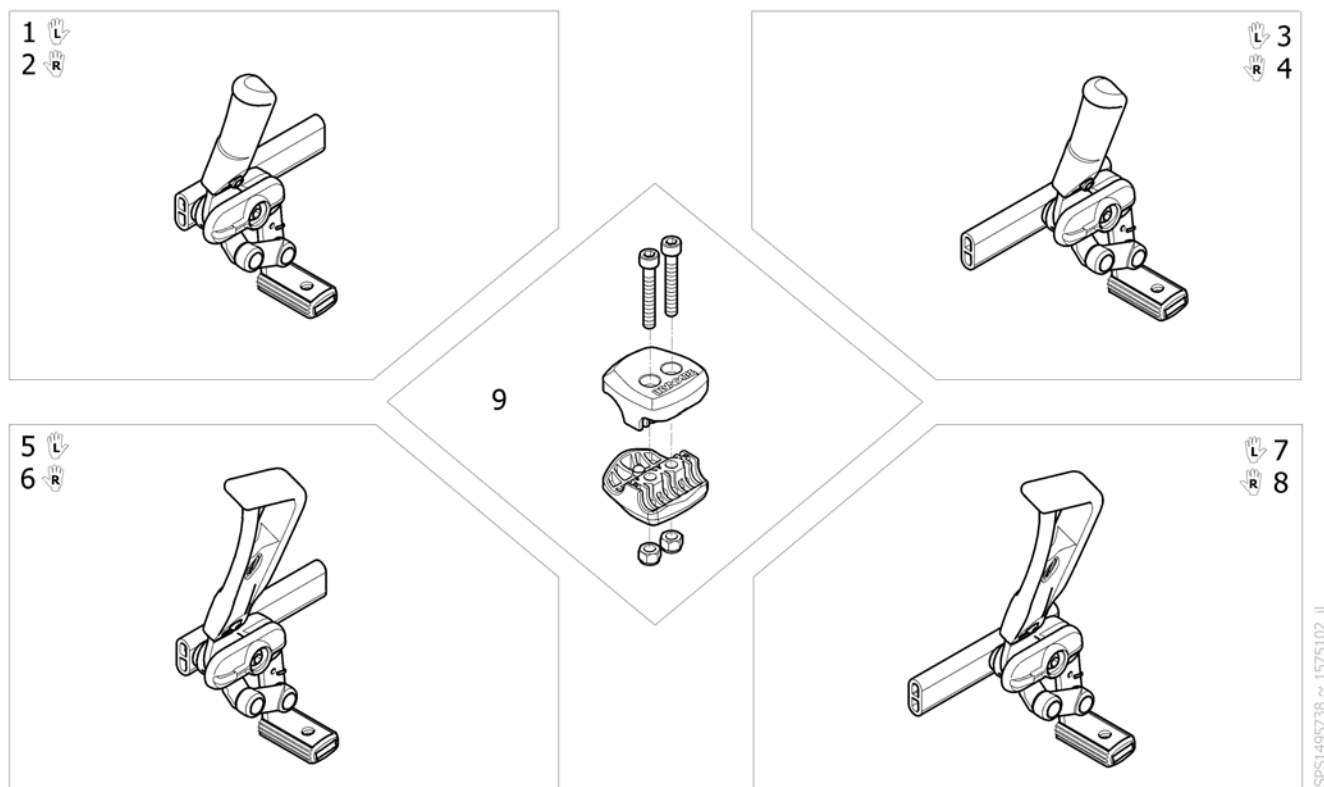


ISPS1495738 ~ 1575102_II

Pos.	Part number	Description	Valid from → until	Qty
1	1425634	Standard push brake Left		1
2	1425635	Standard push brake Right		1
3	1425642	LH Reverse push to lock brake with standard brake-shoe		1
4	1425643	Rh Reverse push to lock brake with standard brake-shoe		1
5	1425602	Lh Complet Brake With Lever Extension		1
6	1425603	Rh Complet Brake With Lever Extension		1
7	1425640	Lh Reverse Complete Brake with lever extension		1
8	1425641	Rh Reverse Complete Brake With Lever Extension		1
9	1575102	Kit brake composite clamp black <i>Replace 1423107</i>		2
9	0572-05715	Kit brake support Alu anodisé incolore		2

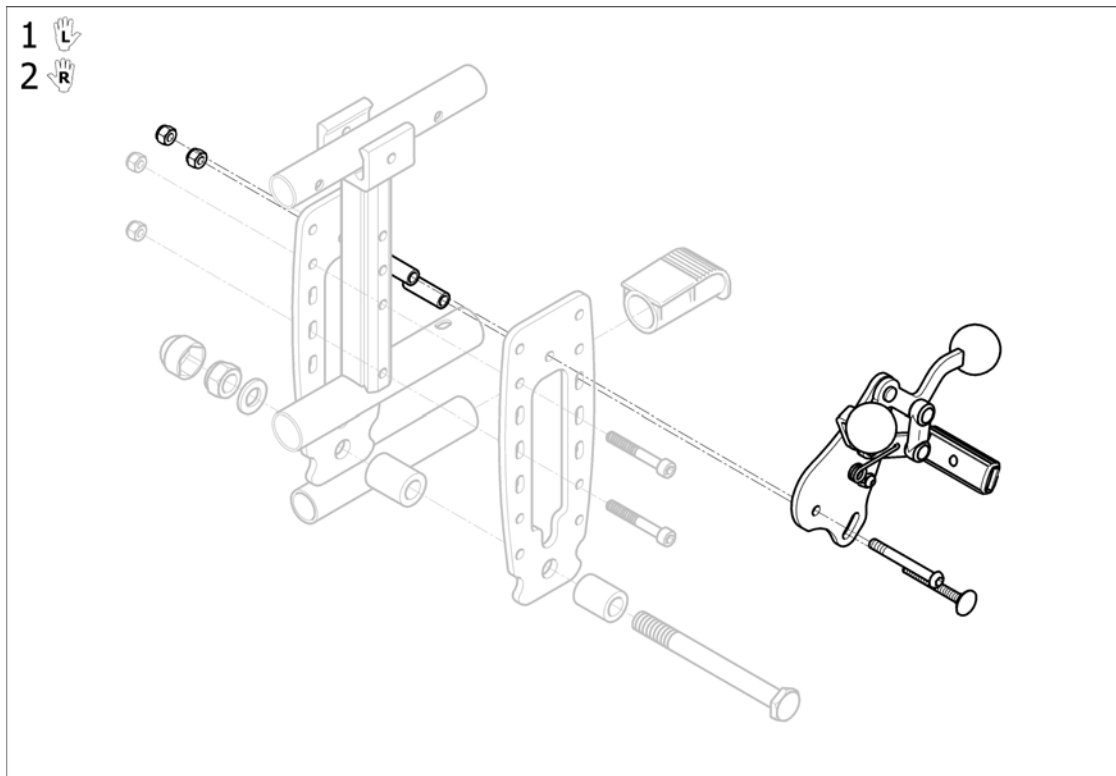Push brake Transit

Pos.	Part number	Description	Valid from → until	Qty
1	1451678	A4 Lh Transit Push Brake		1
2	1451677	A4 Rh Transit Push Brake		1


Pull brakes : Standard And With Extension

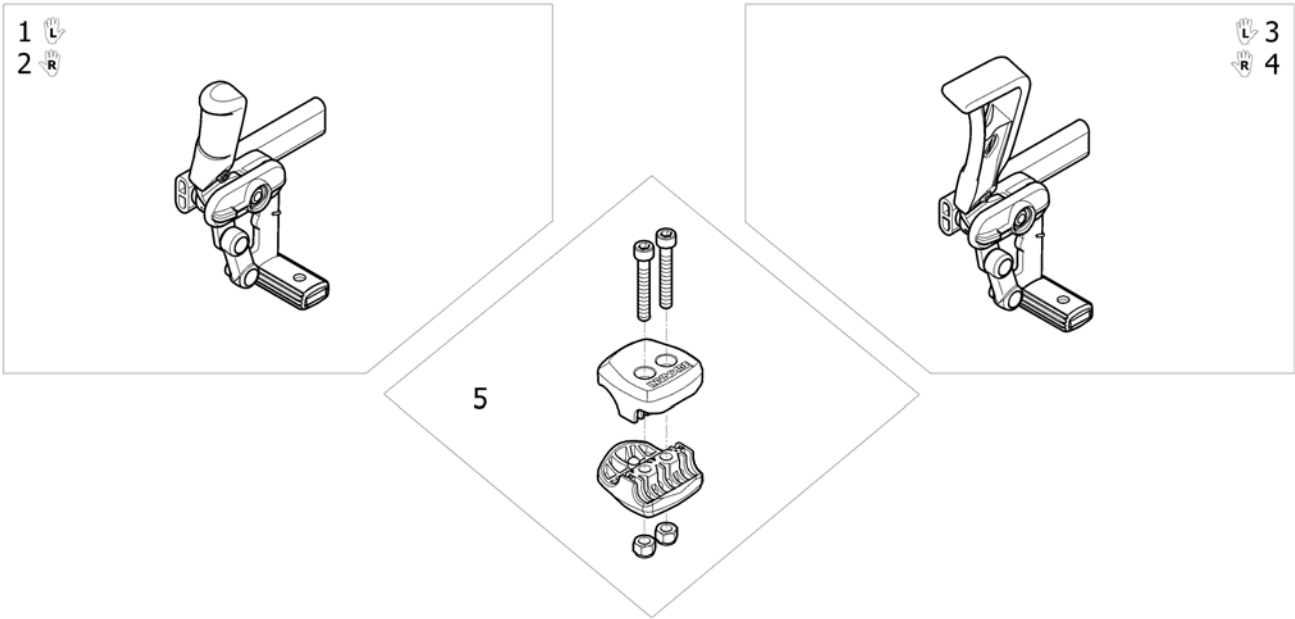

ISPS1548791 ~ sps1548791_II


Pos.	Part number	Description	Valid from → until	Qty
1	1538078	Left Pull brake		1
2	1538079	Right pull brake		1
3	1541956	Left pull brake with lever extension		1
4	1541957	Right pull brake with lever extension		1
5	1575102	Kit brake composite clamp black <i>Replace 1423107</i>		2

Hub Brakes

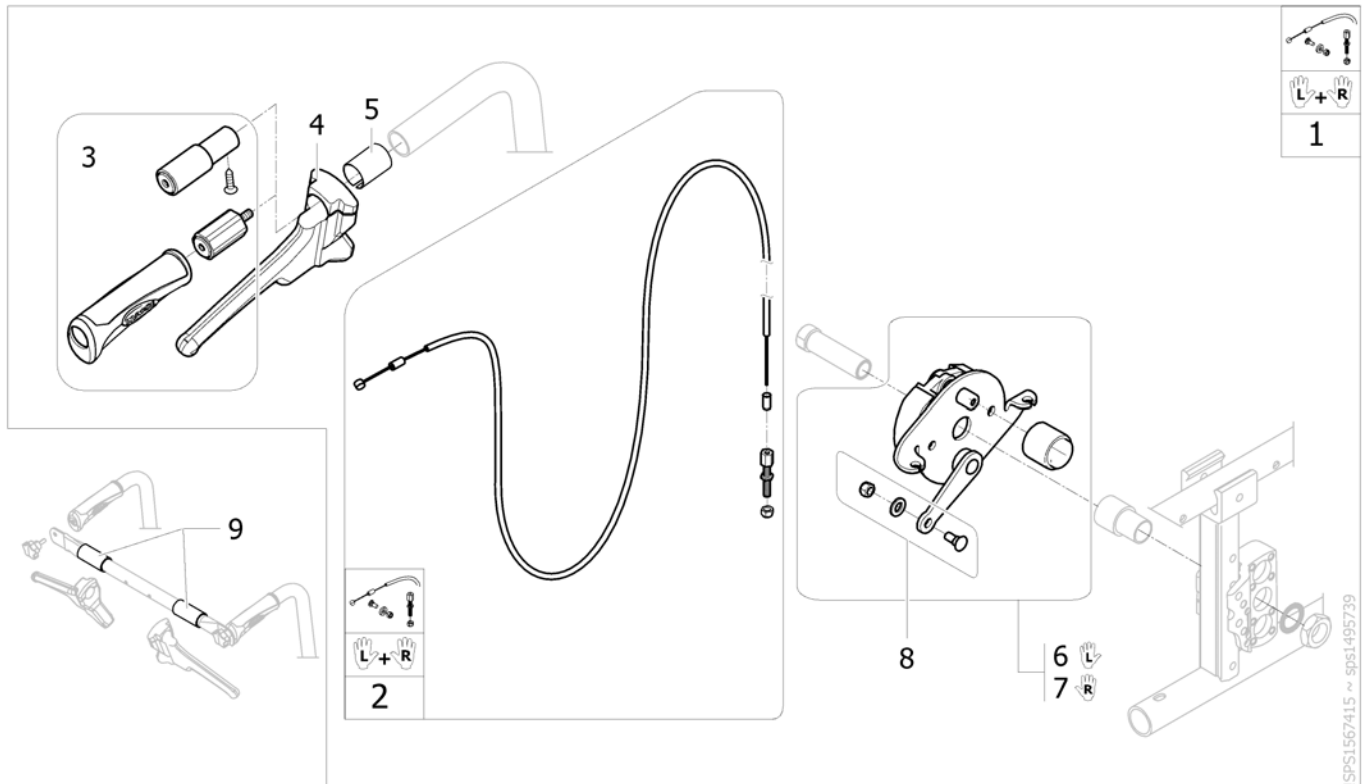

Pos.	Part number	Description	Valid from → until	Qty
1	1567416	Kit pair of attendant hub brakes + tube extension + diameter adapter NG		1
2	1564821	Attendant Hub Brakes Complete Mechanism (pair) NG		1
3	1430412	Back brace insert for fixed backrest tube		1
4	1420527	Black Brake Lever dia 22mm		1
5	5221363	Fixed Backrest Hub Brake Adaptation		2
6	1567413	LH Hub brake plate kit		1
7	1567414	RH Hub brake plate kit		1
8	M30071	Cable Clamp Bolt c/wNut& Wash		2
9	1434626	Backbrace Handle		2



Oad Wire Control Brake

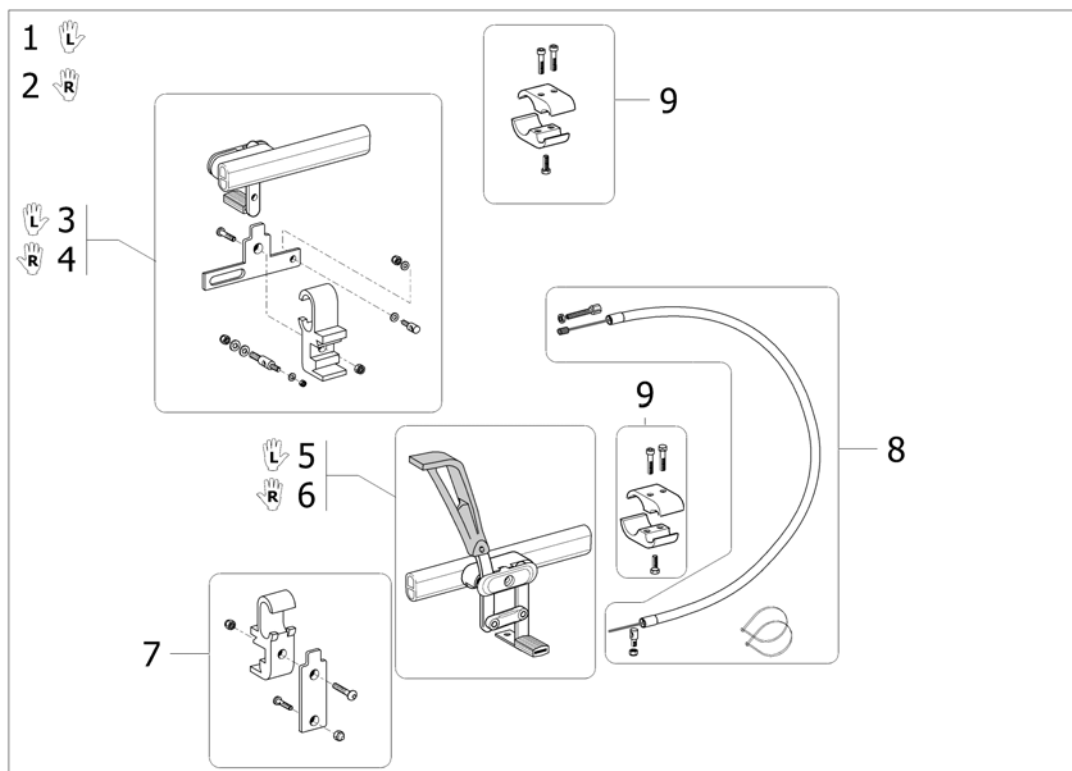

ISPS1495740

Pos.	Part number	Description	Valid from → until	Qty
1	1430203	Lh Oad Brake Wire Controled		1
2	1430202	Rh Oad Brake Wire Controled		1
3	1433461	Lh Wire Control Brake Complete + Clamp; Rh Control		1
4	1433462	Rh Wire Control Brake Complete + Clamp; Lh Control		1
5	1430253	Lh Hemi Brake Action3		1
6	1430254	Rh Hemi Brake Action3		1
7	1433463	Wire Control Brake Clamp Kit		1
8	1433464	Complet Cable Wire Control Brake		1
9	0572-05715	Kit brake support Alu anodisé incolore		2


Safety Options

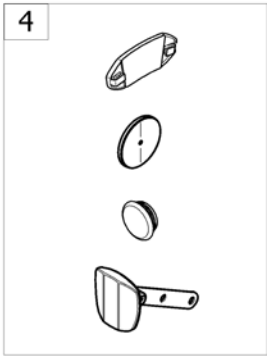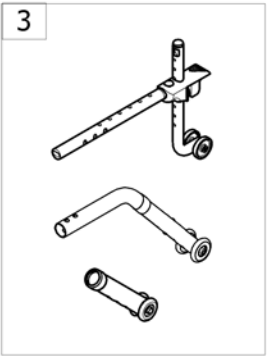

<SPCACTION4NG90

Pos.	Part number	Description	Valid from → until	Qty
1	-	Transportation attachment kit		1
2	-	Belts		1
3	-	Rear Anti Tipper and Tipping Tube		1
4	-	Reflectors		1


Transportation attachment kit

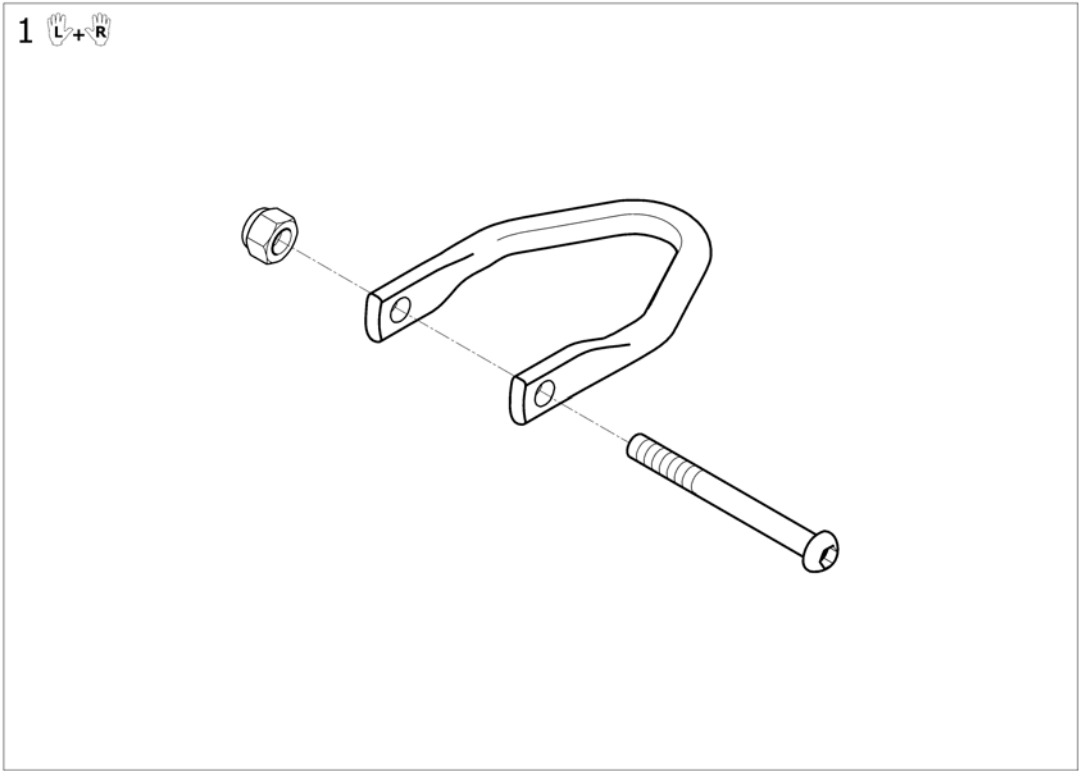

Pos.	Part number	Description	Valid from → until	Qty
1	1560720	Kit transportation (pair)		1

Belts




	Esprit Action 4NG	Action 3NG Action 2000 Variable +	Action 4NG	Action A3J	Action 2NG
SW205	-	-	-	1	-
SW255	-	-	-	1	-
SW280	-	-	-	1	-
SW305	1	-	-	1	-
SW330	1	-	-	1	-
SW355	1	-	-	1	-
SW380	1	1	1	1	3
SW405	1	1	1	-	3
SW430	1	1	1	-	3
SW455	1	1	1	-	3
SW480	2	2	2	-	2
SW505	2	2	2	-	-
SW555	2	-	2	-	-
SW605	2	-	2	-	-

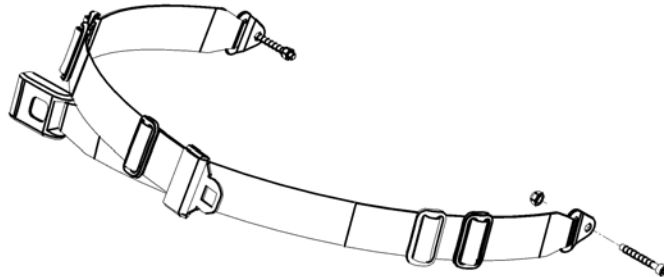

ISPS1497376

Pos.	Part number	Description	Valid from → until	Qty
1	0572-05389	Safety lap belt metal buckle SW205/455 mm		1
2	1492755	Kit safety lap belt metal buckle HD SW480/605 mm		1
3	1531657	Kit safety lap belt metal buckle SW380/455 mm (Action2NG)		1


Rear Anti Tipper and Tipping Tube

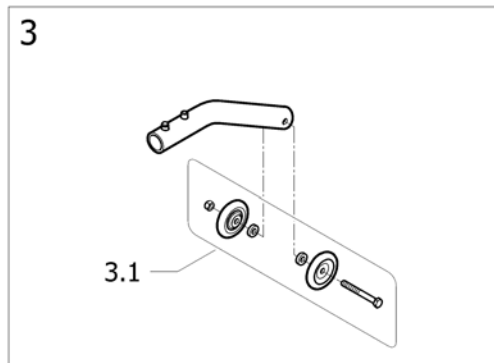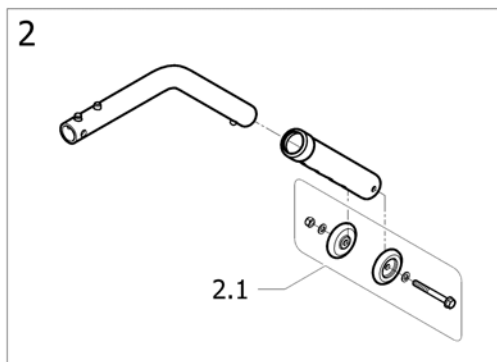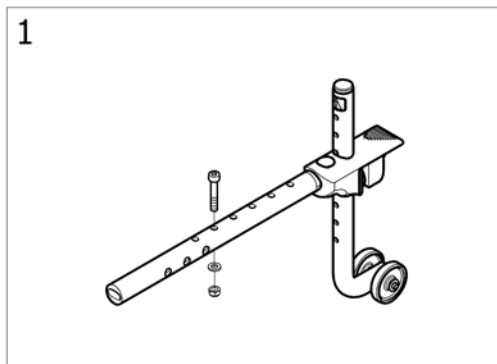

ISPS1548725 ~ sps1548725_jl

Pos.	Part number	Description	Valid from → until	Qty
1	1484890	Anti-tippers A3/A4 types (one side only)		2
2	0572-05848	Kit anti-tippers		2
2.1	1548802	Kit wheels + attachment anti-tippers (washers out)		2
3	1542000	Ultra Low Anti-Tipping		2
3.1	1548248	Kit wheels + attachment anti-tippers (washers in)		2


Reflectors

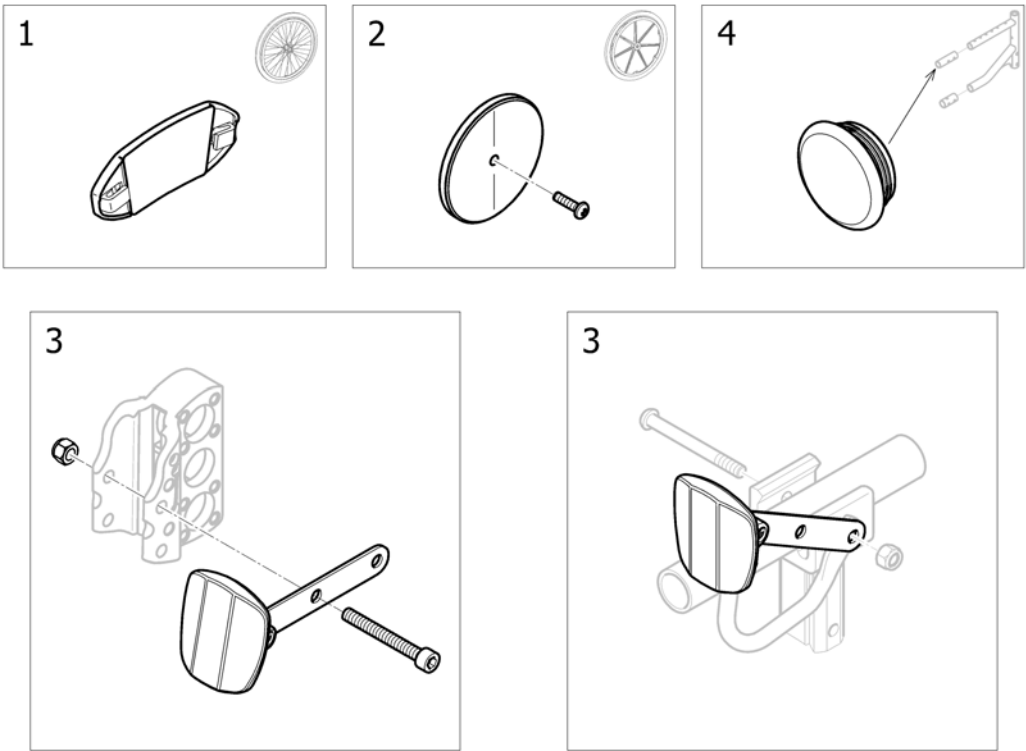

ISPS1496421

Pos.	Part number	Description	Valid from → until	Qty
1	N90003	Kit reflector for spoked wheel		2
2	9037-30001	Kit reflector for composite wheel		2
3	1450569	A4 Frame Reflectors Kit		2
4	1545553	End tube Ø22 reflector red		2

Seating Options

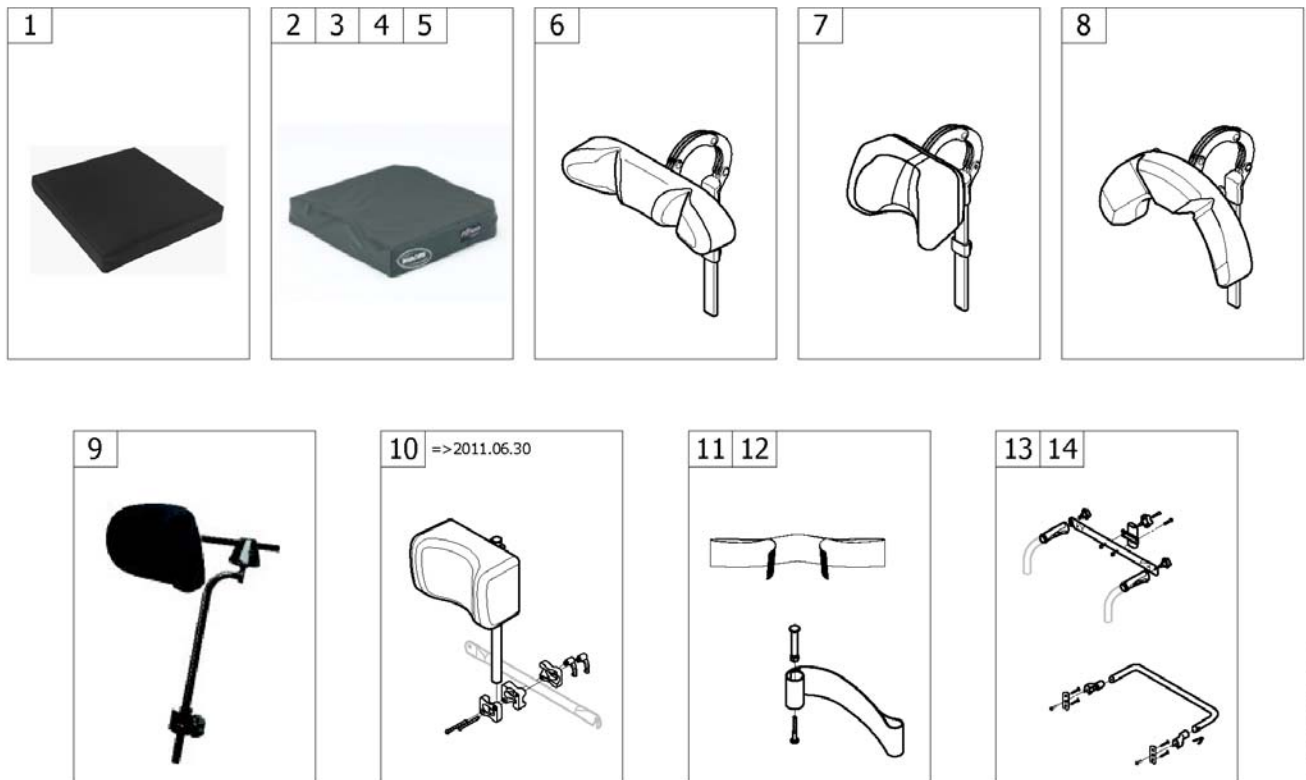

<SPCACTION4NG92

Pos.	Part number	Description	Valid from → until	Qty
1	-	Cushions Nylon [CK_0641]		1
2	-	Cushions Lite [CK_0659]		1
3	-	Cushions Contour Visco [CK_0681]		1
4	-	Cushions Image [CK_0661]		1
5	-	Cushions Solution Xtra [CK_0691]		1
6	-	Headrest [CK_0717]		1
7	-	Neckrest [CK_0716]		1
8	-	Cheek support [CK_0718]		1
9	-	Soft pad headrest and adapter Matrix [CK_0719]		1
10	-	Standard Headrest =>2011-06-30	→ 2011-06-30	1
11	-	Calf Strap [CK_0760]		1
12	-	Heels strap [CK_0750]		1
13	-	Back Brace standard + support [CK_0780]		1
14	-	Back Brace type Rea [CK_0780]		1

Cushions Nylon

	Action 3NG EspritA4NG		Action 4NG		Action 2000 Action 2NG		Action 1NG		Variable
	SD400	SD450	SD400	SD450	SD400	SD450	SD400	SD450	
SW380	1	7	1	7	1	7	-	-	1
SW405	2	8	2	8	2	8	2	8	2
SW430	3	9	3	9	3	9	3	9	3
SW455	4	10	4	10	4	10	4	10	4
SW480	5	11	5	11	5	11	5	11	5
SW505	6	12	6	12	-	-	6	12	6
SW555	-	-	-	13	-	-	-	-	-
SW605	-	-	-	14	-	-	-	-	-



ISPS1551050

Pos.	Part number	Description	Valid from → until	Qty
1	SP1586069-3840	Cushion seat foam Nylon SW380 SD400 <i>Replace 1541977</i>		1
2	SP1586069-4040	Cushion seat foam Nylon SW405 SD400 <i>Replace 1541977</i>		1
3	SP1586069-4340	Cushion seat foam Nylon SW430 SD400 <i>Replace 1541979</i>		1
4	SP1586069-4540	Cushion seat foam Nylon SW455 SD400 <i>Replace 1541979</i>		1
5	SP1586069-4840	Cushion seat foam Nylon SW480 SD400 <i>Replace 1541981</i>		1
6	SP1586069-5040	Cushion seat foam Nylon SW505 SD400 <i>Replace 1541981</i>		1
7	SP1586069-3845	Cushion seat foam Nylon SW380 SD450 <i>Replace 1541978</i>		1
8	SP1586069-4045	Cushion seat foam Nylon SW405 SD450 <i>Replace 1541978</i>		1
9	SP1586069-4345	Cushion seat foam Nylon SW430 SD450 <i>Replace 1541980</i>		1
10	SP1586069-4545	Cushion seat foam Nylon SW455 SD450 <i>Replace 1541980</i>		1
11	SP1586069-4845	Cushion seat foam Nylon SW480 SD450 <i>Replace 1541982</i>		1
12	SP1586069-5045	Cushion seat foam Nylon SW505 SD450 <i>Replace 1541982</i>		1
13	SP1586069-5545	Cushion seat foam Nylon SW555 SD450 <i>Replace 1548447</i>		1
14	SP1586069-6045	Cushion seat foam Nylon SW605 SD450 <i>Replace 1548448</i>		1

Cushions Lite

	Action 3NG Action 4NG EspritA4NG	
	SD400	SD450
SW380	1	7
SW405	2	8
SW430	3	9
SW455	4	10
SW480	5	11
SW505	6	12



ISPS1574850

Pos.	Part number	Description	Valid from → until	Qty
1	1575607	Cushion Lite, SW380 x SD400mm		1
2	1575609	Cushion Lite, SW405 x SD400mm		1
3	1575611	Cushion Lite, SW430 x SD400mm		1
4	1575613	Cushion Lite, SW455 x SD400mm		1
5	1575615	Cushion Lite, SW480 x SD400mm		1
6	1575617	Cushion Lite, SW505 x SD400mm		1
7	1575608	Cushion Lite, SW380 x SD450mm		1
8	1575610	Cushion Lite, SW405 x SD450mm		1
9	1575612	Cushion Lite, SW430 x SD450mm		1
10	1575614	Cushion Lite, SW455 x SD450mm		1
11	1575616	Cushion Lite, SW480 x SD450mm		1
12	1575618	Cushion Lite, SW505 x SD450mm		1

Cushions Contour Visco

	Action 3NG Action 4NG EspritA4NG		Action 2NG
	SD400	SD450	SD400
SW380	1	7	1
SW405	2	8	2
SW430	3	9	3
SW455	4	10	4
SW480	5	11	5
SW505	6	12	-



1523882

Pos.	Part number	Description	Valid from → until	Qty
1	1559590	Cushion Contour Visco, SW380 x SD405 x H90mm MUW:100Kg (FCSB18/18/LMF/SGL) <i>Replace 1501111</i>		1
2	1559659	Cushion Contour Visco, SW405 x SD405 x H90mm MUW:100Kg (FLSB16/16/LMF/SGL) <i>Replace 1501109</i>		1
3	1559730	Cushion Contour Visco, SW430 x SD405 x H90mm MUW:100Kg (FLSB17/16/LMF/SGL) <i>Replace 1501107</i>		1
4	1501114	Cushion Contour Visco, SW455 x SD405 x H90mm MUW:100Kg (FCSB18/16/LMF/SGL) <i>Replace 1523826</i>		1
5	1501116	Cushion Contour Visco, SW480 x SD405 x H90mm MUW:100Kg (FCSB19/16/LMF/SGL)		1
6	1501118	Cushion Contour Visco, SW505 x SD405 x H90mm MUW:125Kg (FCSB20/16/LMF/SGL)		1
7	1559657	Cushion Contour Visco, SW380 x SD455 x H90mm MUW:100Kg (FLSB15/18/LMF/SGL) <i>Replace 1501112</i>		1
8	1526594	Cushion Contour Visco, SW405 x SD455 x H90mm MUW:100Kg (FCSB16/18/LMF/SGL) <i>Replace 1501110</i>		1
9	1526595	Cushion Contour Visco, SW430 x SD455 x H90mm MUW:100Kg (FCSB17/18/LMF/SGL) <i>Replace 1501108</i>		1
10	1526608	Cushion Contour Visco, SW455 x SD455 x H90mm MUW:100Kg (FCSB18/18/LMF/SGL) <i>Replace 1501115/ 1525535</i>		1
11	1525020	Cushion Contour Visco, SW480 x SD455 x H90mm MUW:125Kg (FCSB19/18/LMF/SGL) <i>Replace 1501117</i>		1
12	1525021	Cushion Contour Visco, SW505 x SD455 x H90mm MUW:125Kg (FCSB20/18/LMF/SGL) <i>Replace 1501119</i>		1

Cushions Image



15PS 1528883

Pos.	Part number	Description	Valid from → until	Qty
1	1575728	Cushion Image, SW380 x SD405 x H78mm, MUW-140kg (FISB15/16)		1
2	1511196	Cushion Image, SW405 x SD405 x H78mm, MUW-140kg (FISB16/16)		1
3	1511197	Cushion Image, SW430 x SD405 x H78mm, MUW-140kg (FISB17/16)		1
4	1511198	Cushion Image, SW455 x SD405 x H78mm, MUW-140kg (FISB18/16)		1
5	1511199	Cushion Image, SW480 x SD405 x H78mm, MUW-140kg (FISB19/16)		1
6	1526417	Cushion Image, SW505 x SD405 x H78mm, MUW-140kg (FISB20/16)		1
7	1575729	Cushion Image, SW380 x SD455 x H78mm, MUW-140kg (FISB15/18)		1
8	1526412	Cushion Image, SW405 x SD455 x H78mm, MUW-140kg (FISB16/18) <i>Replace 1523771</i>		1
9	1526413	Cushion Image, SW430 x SD455 x H78mm, MUW-140kg (FISB17/18) <i>Replace 1511200</i>		1
10	1526414	Cushion Image, SW455 x SD455 x H78mm, MUW-140kg (FISB18/18) <i>Replace 1523772</i>		1
11	1526415	Cushion Image, SW480 x SD455 x H78mm, MUW-140kg (FISB19/18) <i>Replace 1523813</i>		1
12	1526416	Cushion Image, SW505 x SD455 x H78mm, MUW-140kg (FISB20/18)		1

Cushions Solution Xtra


	Action 3NG Action 4NG Espira4NG	
	SD400	SD450
SW380	1	7
SW405	2	8
SW430	3	9
SW455	4	10
SW480	5	11
SW505	6	12



ISPS1574851

Pos.	Part number	Description	Valid from → until	Qty
1	1494430	Cushion Solution Xtra, SW380 x SD405mm		1
2	1487145	Cushion Solution Xtra, SW405 x SD405mm		1
3	1494441	Cushion Solution Xtra, SW430 x SD405mm		1
4	1487147	Cushion Solution Xtra, SW455 x SD405mm		1
5	1494451	Cushion Solution Xtra, SW480 x SD405mm		1
6	1552058	Cushion Soution Xtra, SW505 x SD405mm		1
7	1494432	Cushion Solution Xtra, SW380 x SD455mm		1
8	1551914	Cushion Solution Xtra, SW405 x SD455mm		1
9	1551922	Cushion Solution Xtra, SW430 x SD455mm		1
10	1551925	Cushion Solution Xtra, SW455 x SD455mm		1
11	1552055	Cushion Solution Xtra, SW480 x SD455mm		1
12	1552060	Cushion Solution Xtra, SW505 x SD455mm		1


Headrest

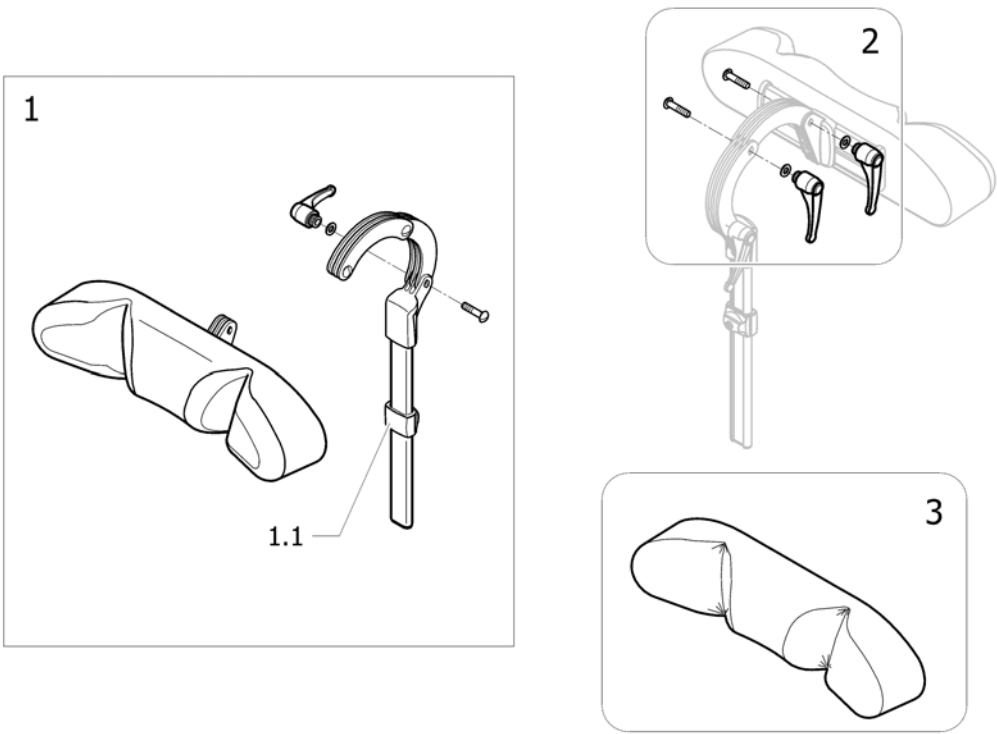

ISPS1553018 ~ sps1553018_II

Pos.	Part number	Description	Valid from → until	Qty
1	1446559	HEADREST TR26 CPL SIDEWISE ADJUSTABLE		1
1.1	1448661	stop neck and headrest		1
2	1445693	HANDLES, CLINIC KIT NECK- AND HEADREST		1
3	1518516	Cover for headrest TR26		1



Neckrest

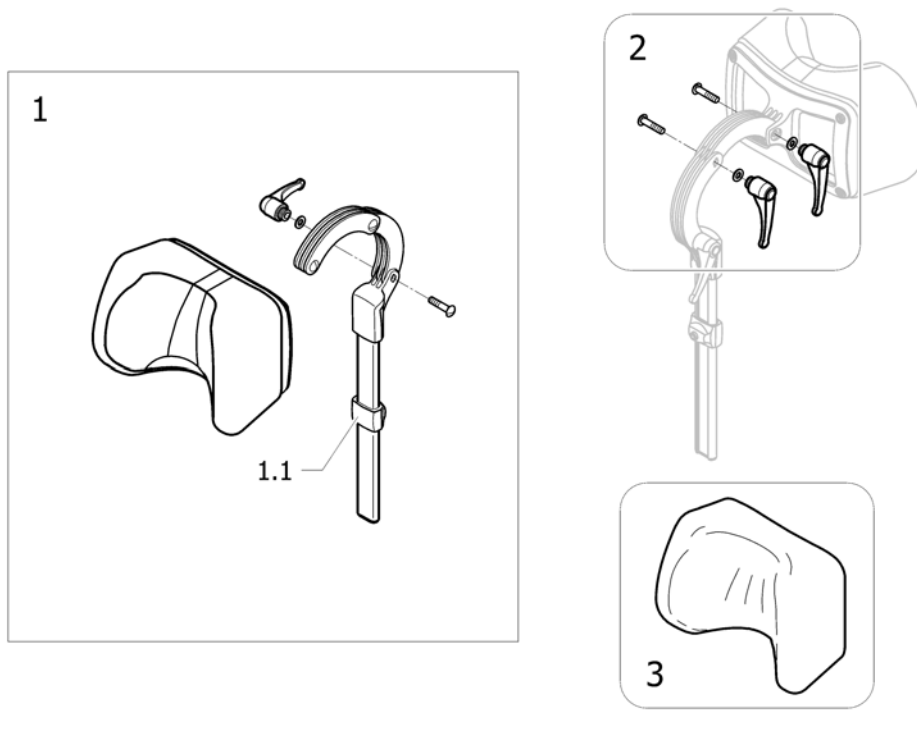

!SPS1548246

Pos.	Part number	Description	Valid from → until	Qty
1	1508848	NECKREST, CPL		1
1.1	1448661	stop neck and headrest		1
2	1445693	HANDLES, CLINIC KIT NECK- AND HEADREST		1
3	1518517	Cover neckrest TR34		1



Cheek support

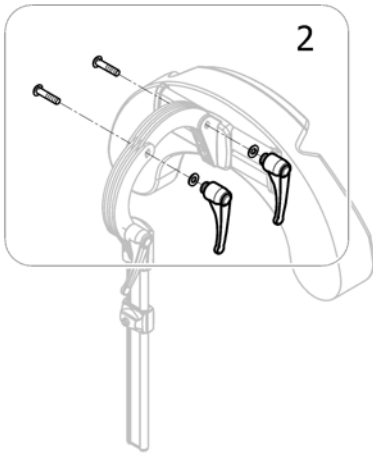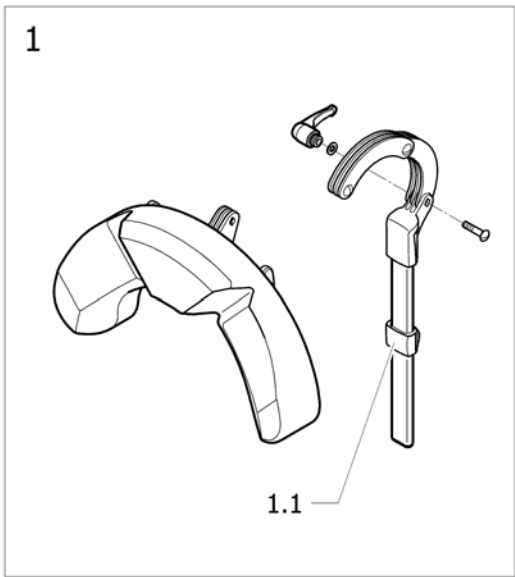

ISPS1555289 ~ sps1553018_II

Pos.	Part number	Description	Valid from → until	Qty
1	1446560	HEAD/CHEEK SUPP. TR26 CPL. SIDEWISE ADJUSTABLE		1
1.1	1448661	stop neck and headrest		1
2	1445693	HANDLES, CLINIC KIT NECK- AND HEADREST		1



Soft pad headrest and adapter Matrix

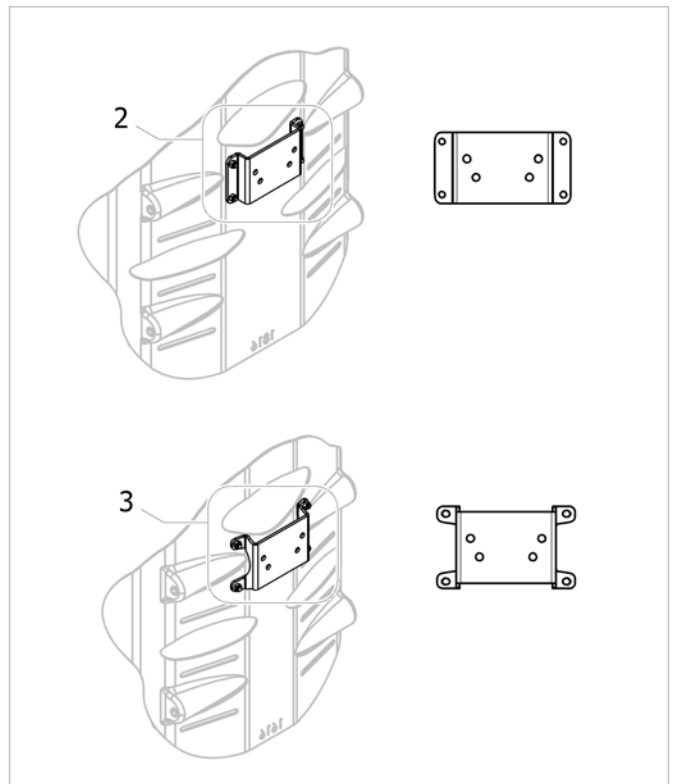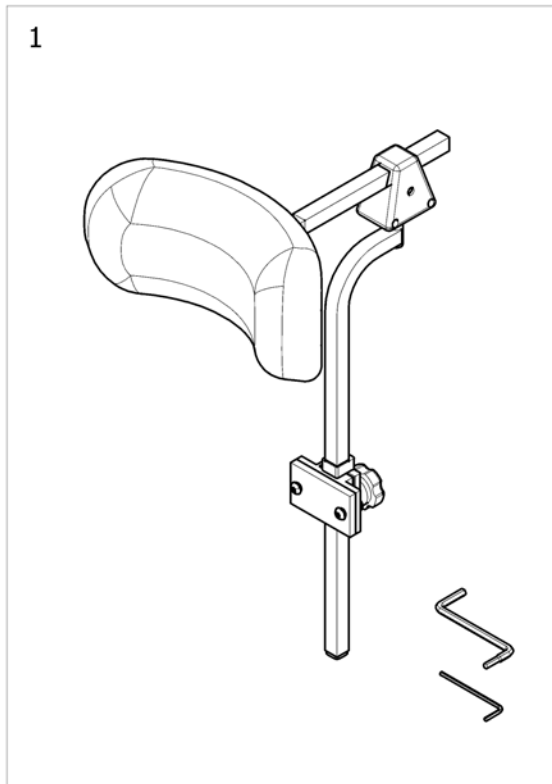

Pos.	Part number	Description	Valid from → until	Qty
1	1545274	Matrix 255 mm Soft Pad Headrest Kit (including fixation for PB and PB deep)		1
2	1544187	Elite Headrest Mounting Hardware - Large		1
3	1544188	Elite Headrest Mounting Hardware - Medium		1

Standard Headrest =>2011-06-30


	Action 4NG	Action 3NG	Action 2NG
SW380	1	1	1
SW405	2	2	2
SW430	3	3	3
SW455	4	4	4
SW480	5	5	5
SW505	6	6	-
SW555	7	-	-
SW605	8	-	-

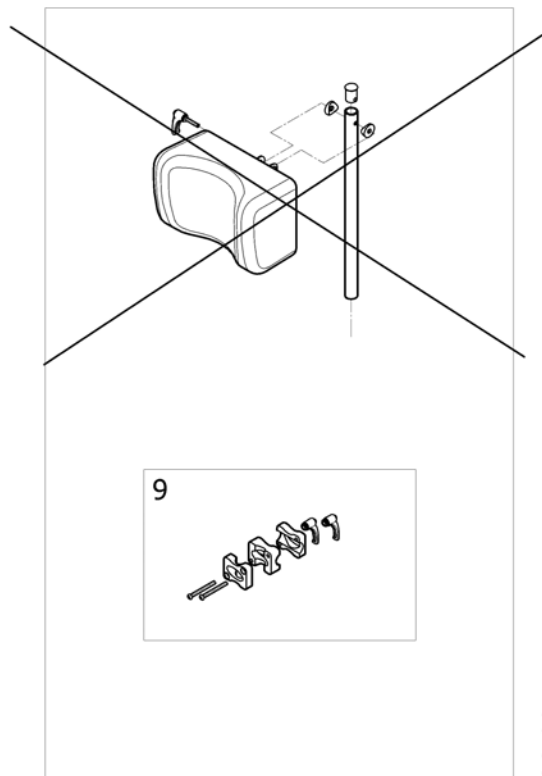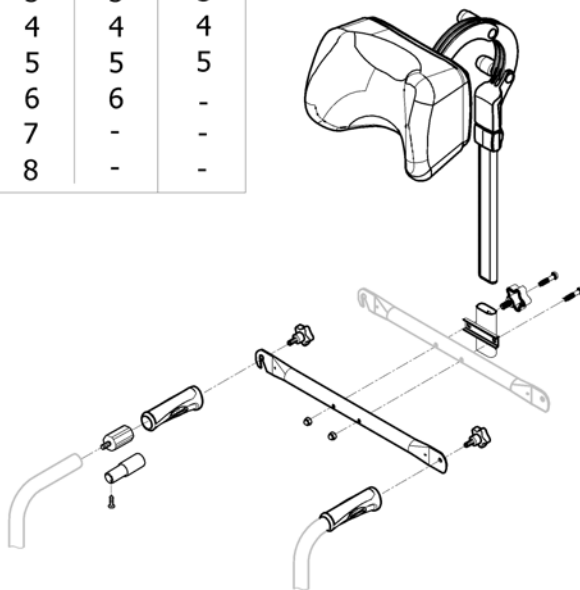

ISPS1497375


Pos.	Part number	Description	Valid from → until	Qty
1	1574745	Headrest + Backbrace NG SW380 mm <i>Replace 1433028 (1432623+1432622+1432617+1434099)</i>	→ 2011-06-30	1
2	1574746	Headrest + Backbrace NG SW405 mm <i>Replace 1433029 (1432623+1432622+1432617+1434100)</i>	→ 2011-06-30	1
3	1574747	Headrest + Backbrace NG SW430 mm <i>Replace 1433030 (1432623+1432622+1432617+1433707)</i>	→ 2011-06-30	1
4	1574748	Headrest + Backbrace NG SW455 mm <i>Replace 1433031 (1432623+1432622+1432617+1433708)</i>	→ 2011-06-30	1
5	1574749	Headrest + Backbrace NG SW480 mm <i>Replace 1433032 (1432623+1432622+1432617+1433716)</i>	→ 2011-06-30	1
6	1574763	Headrest + Backbrace NG SW505 mm <i>Replace 1433033 (1432623+1432622+1432617+1434105)</i>	→ 2011-06-30	1
7	1574806	Headrest + Backbrace NG SW555 mm <i>Replace 1496214 (1432623+1432622+1432617+1483910)</i>	→ 2011-06-30	1
8	1574807	Headrest + Backbrace NG SW605 mm <i>Replace 1496215 ((1432623+1432622+1432617+1483911)</i>	→ 2011-06-30	1
9	1432617	Headrest Support Kit	→ 2011-06-30	1

Calf Strap

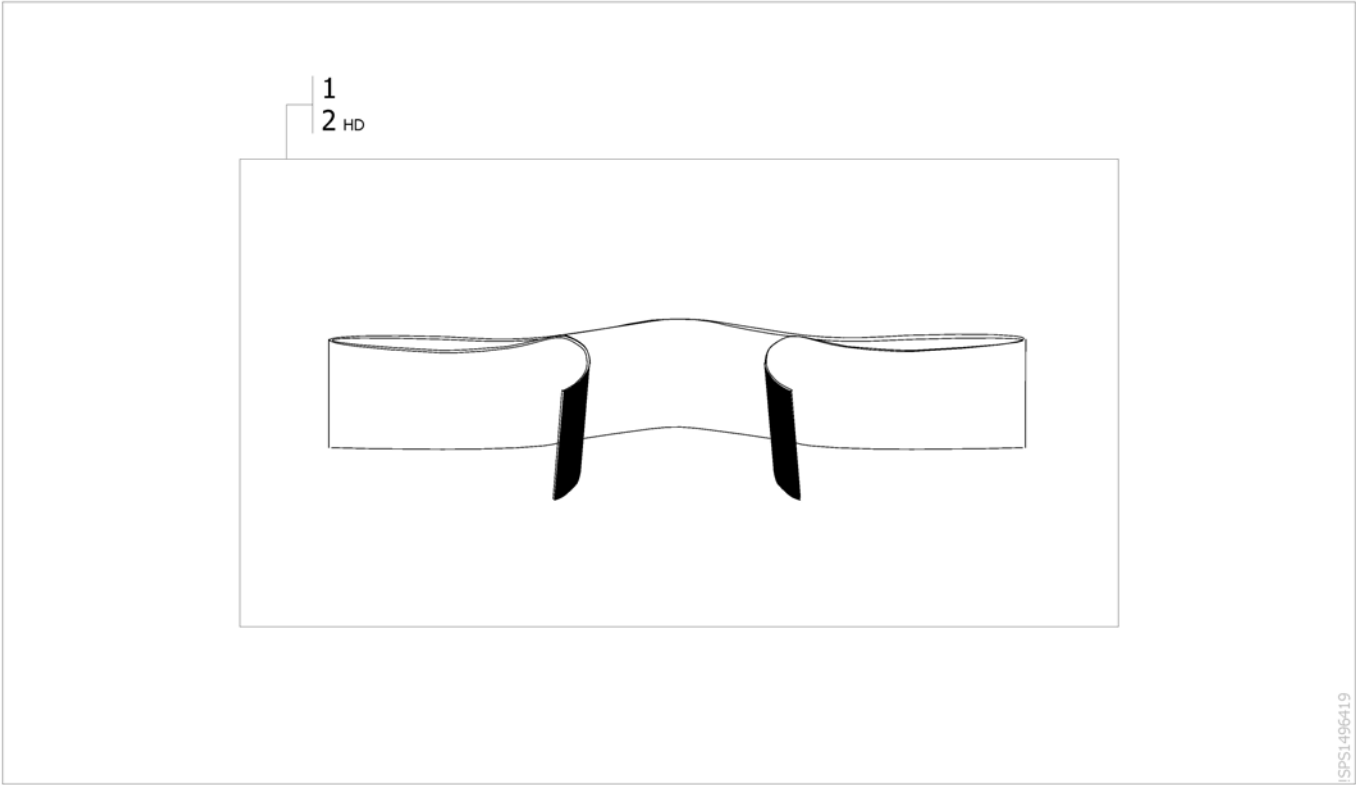


Pos.	Part number	Description	Valid from → until	Qty
1	1433580	Adjustable Calf Strap SW 380 to 408 mm		1
2	1497574	Calf strap adjustable, large, SW505-605mm		1

Heels strap

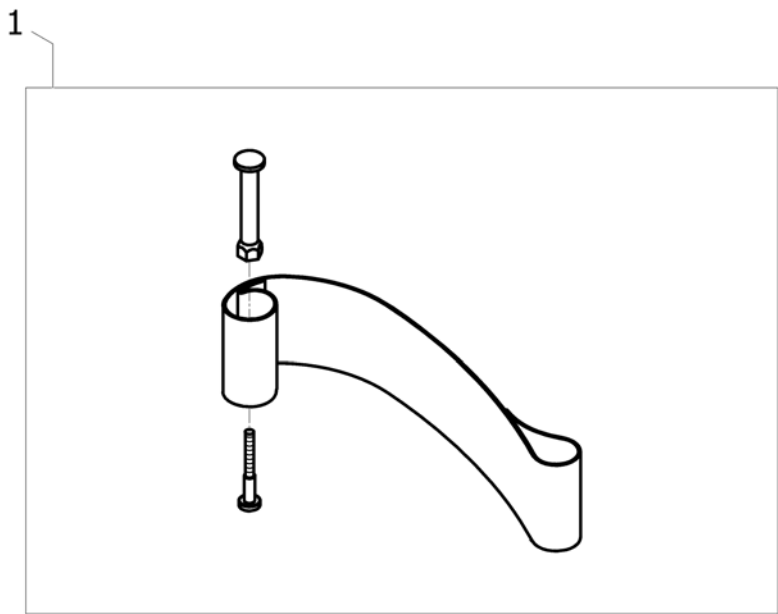

159133490



Pos.	Part number	Description	Valid from → until	Qty
1	1492315	Kit heelstrap, adjustable, medium <i>Replace 1534891</i>		2

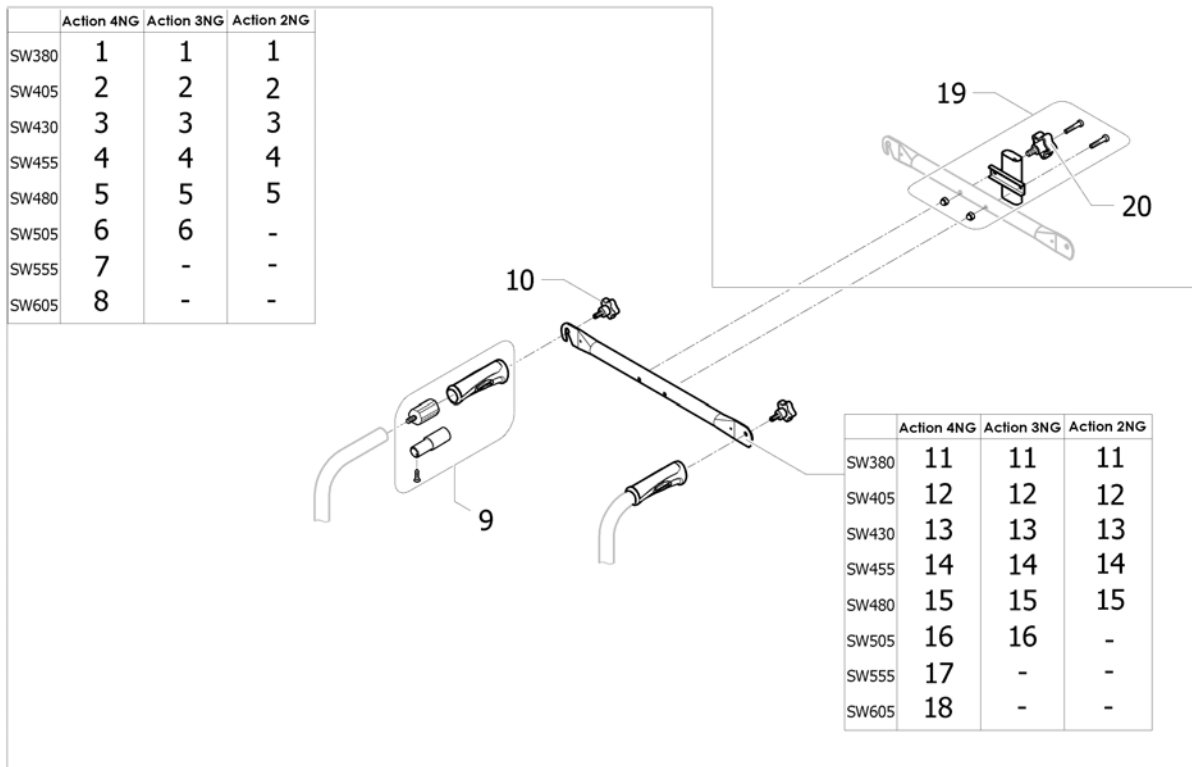Back Brace standard + support

ISPS1543302 ~ sps1553018_II

Pos.	Part number	Description	Valid from → until	Qty
1	1434099	Back Brace with inserts and handles SW380 mm		1
2	1434100	Back Brace with inserts and handles SW405 mm		1
3	1433707	Back Brace with inserts and handles SW430 mm		1
4	1433708	Back Brace with inserts and handles SW455 mm		1
5	1433716	Back Brace with inserts and handles SW480 mm		1
6	1434105	Back Brace with inserts and handles SW505 mm		1
7	1483910	Back Brace with inserts and handles SW555 mm		1
8	1483911	Back Brace with inserts and handles SW605 mm		1
9	1430412	Back brace insert for fixed backrest tube		2
10	0021-60029	Button Ø32 mm M6x16		2
11	1428535-0601	Backbrace SW380 mm black RAL9005		1
12	1428536-0601	Backbrace SW405 mm black RAL9005		1
13	1428537-0601	Backbrace SW430 mm black RAL9005		1
14	1428538-0601	Backbrace SW455 mm black RAL9005		1
15	1428539-0601	Backbrace SW480 mm black RAL9005		1
16	1428540-0601	Backbrace SW505 mm black RAL9005		1
17	1485926-0601	Backbrace SW555 mm black RAL9005		1
18	1485927-0601	Backbrace SW605 mm black RAL9005		1
19	1554616	Kit headrest support (bracket+screw+bolt+button screw M8) Action NG	2011-07-01 →	1
20	0021-60026	Button screw 40 mm M8x16	2011-07-01 →	1






Back Brace type Rea

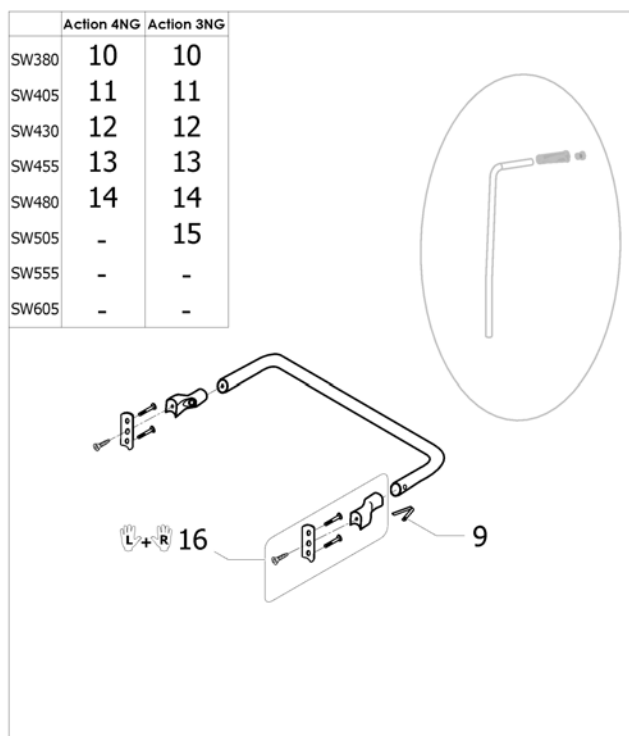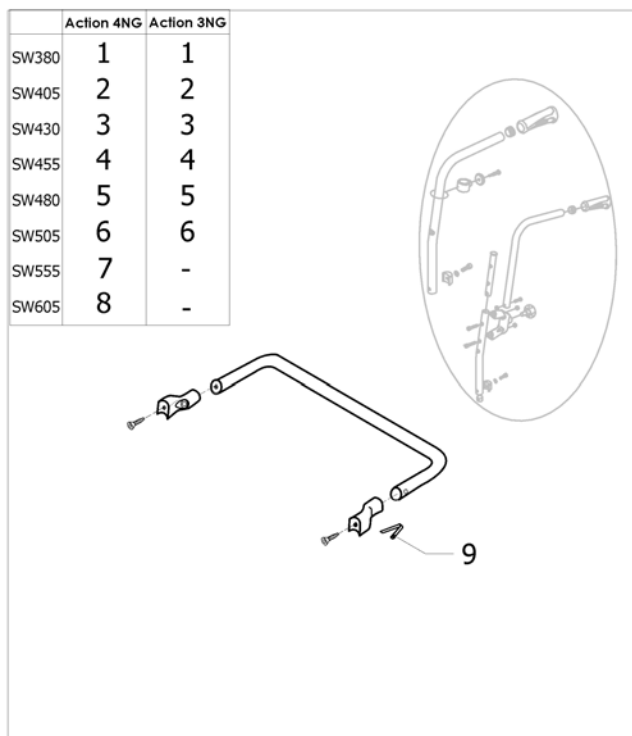

ISPS1496420 ~ sps1553018_II

Pos.	Part number	Description	Valid from → until	Qty
1	1450575	Backbrace type Rea fix SW380 mm [CE_0395/CE_0396] [CK_0300/CK_0310]		1
2	1450576	Backbrace type Rea fix SW405 mm [CE_0395/CE_0396] [CK_0300/CK_0310]		1
3	1450577	Backbrace type Rea fix SW430 mm [CE_0395/CE_0396] [CK_0300/CK_0310]		1
4	1450578	Kit Backbrace SW455 for fixed backrest [CE_0395/CE_0396] [CK_0300/CK_0310]		1
5	1450562	Backbrace type Rea fix SW480 mm [CE_0395/CE_0396] [CK_0300/CK_0310]		1
6	1450451	Backbrace type Rea fix SW505 mm [CE_0395/CE_0396] [CK_0300/CK_0310]		1
7	1451482	Backbrace type Rea fix SW555 mm [CK_0300/CK_0310]		1
8	1451483	Backbrace type Rea fix SW605 mm [CK_0300/CK_0310]		1
9	0027-10003	Spring pin locked armrest		1
10	1449033	Back Brace SW380 mm Type Rea rack [CE_0428] [CK_0315]		1
11	1449034	Back Brace SW405 mm Type Rea rack [CE_0428] [CK_0315]		1
12	1449035	Back Brace SW430 mm Type Rea rack [CE_0428] [CK_0315]		1
13	1451225	Back Brace SW455 mm Type Rea rack [CE_0428] [CK_0315]		1
14	1451226	Back Brace SW480 mm Type Rea rack [CE_0428] [CK_0315]		1
15	1451227	Back Brace SW505 mm Type Rea rack [CE_0428]		1
16	1493360	Rea Backbrace Fixation Kit (pair)		1

Propulsion Options

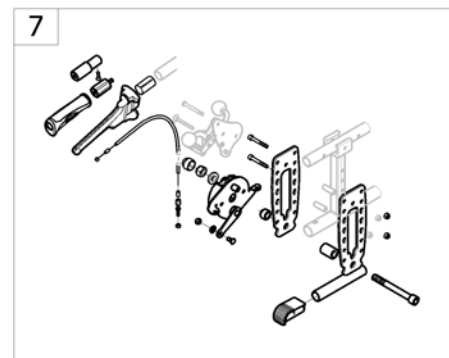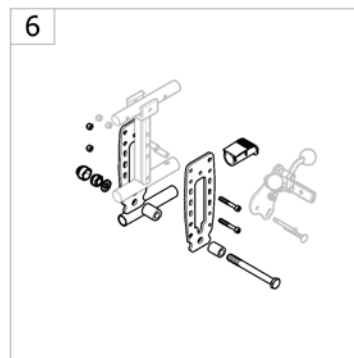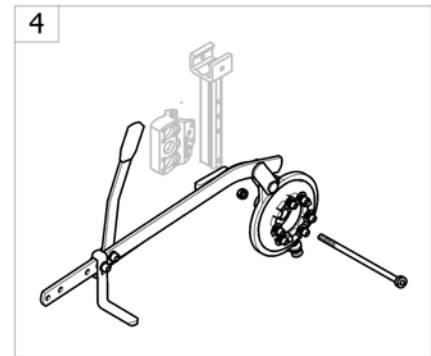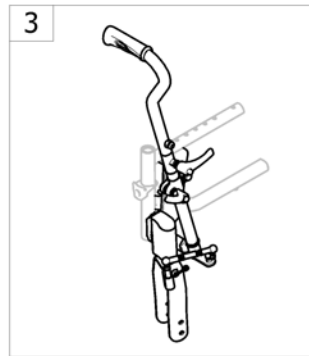
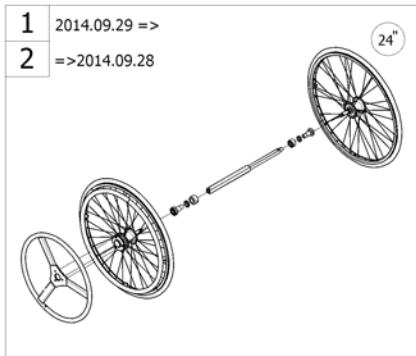


<SPCACTION4NGS6

Pos.	Part number	Description	Valid from → until	Qty
1	-	Dual handrim QR axle 2014.09.29 ==>		1
2	-	Dual handrim QR axle ==> 2014.09.28	→ 2014-09-28	1
3	-	Hand Control For Oad By Lever		1
4	-	Oad By Lever		1
5	-	Wheels For Oad By Lever		1
6	-	Rear wheel transit supports for manual brakes		1
7	-	Rear wheel transit supports for hub brakes		1

Dual handrim QR axle 2014.09.29 =>

	Action3NG Action4NG		Action3NG Double Amputé		Action1NG	
						
	1.3/8"	1.3/8"	1.3/8"	1.3/8"	1.3/8"	1.3/8"
SW380	1	3	1	3	-	-
SW405	1	3	1	3	1	3
SW430	1	3	2	4	2	4
SW455	1	3	2	4	2	4
SW480	2	4	2	4	2	4
SW505	2	4	2	4	2	4

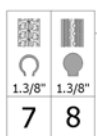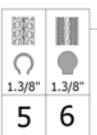



	A3NG A4NG	A3NG Double Amputé	A1NG
SW380	10	10	-
SW405	10	10	10
SW430	10	11	11
SW455	10	11	11
SW480	11	11	11
SW505	11	11	11

SP1585328


Pos.	Part number	Description	Valid from → until	Qty
1	SP1585323	Dual handrim DHR pneumatic kit assy SW380/405/430/455 mm		1
2	SP1585324	Dual handrim DHR pneumatic kit assy SW430/455/480/505 mm		1
3	SP1585325	Dual handrim DHR solid tyre kit assy SW380/405/430/455 mm		1
4	SP1585326	Dual handrim DHR solid tyre kit assy SW430/455/480/505 mm		1
5	SP1572681	24" DHR Passive wheel, pneumatic + axle		1
6	SP1572682	24" DHR Passive wheel, solid tyre + axle		1
7	SP1572683	24" DHR Active wheel, pneumatic + axle + small handrim		1
8	SP1572684	24" DHR Active wheel, solid tyre + axle + small handrim		1
9	SP1572685	Wheel bearing for DHR		2
10	SP1572544	Telescopic shaft "A" for DHR, mini=259mm, maxi=359mm		1
11	SP1572545	Telescopic shaft "B" for DHR, mini=340mm, maxi=438mm		1
12	SP1585327	Small handrim + screw + sticker		1
13	SP1585328	24" Aluminium handrim + 6 fixations		1

Invacare® Action 4 NG | Propulsion Options
Dual handrim QR axle => 2014.09.28

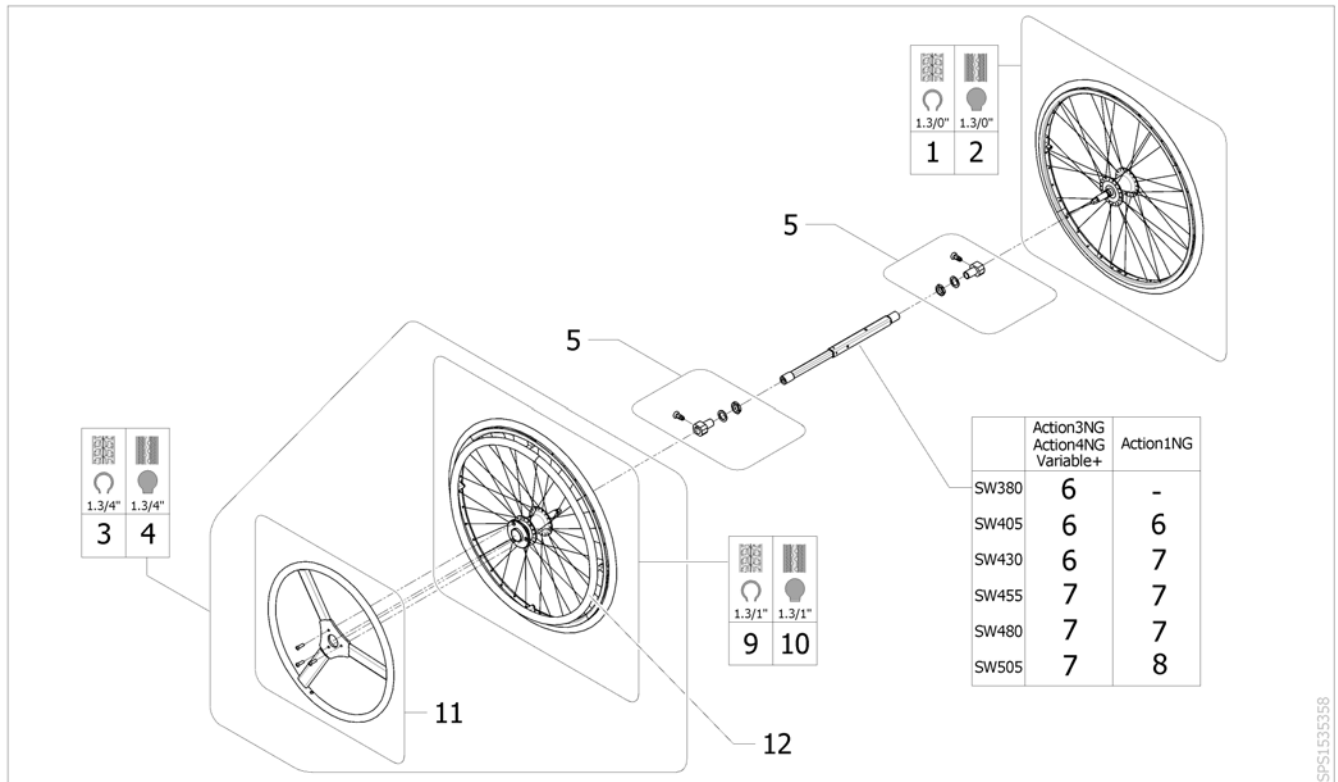

Pos.	Part number	Description	Valid from → until	Qty
1	1508491	24" DHR passive wheel pneumatic	→ 2014-09-28	1
2	1508490	24" DHR passive wheel solid	→ 2014-09-28	1
3	1509382	24" DHR active wheel pneumatic	→ 2014-09-28	1
4	1509436	24" DHR active wheel solid	→ 2014-09-28	1
5	1513658	DHR wheel axle support complet	→ 2014-09-28	2
6	1511769	Telescopic Shaft 270 mm -385mm	→ 2014-09-28	1
7	1508487	Telescopic Shaft 345 mm -460mm	→ 2014-09-28	1
8	1511770	Telescopic Shaft 375 mm -495mm	→ 2014-09-28	1
9	1508485	24" DHR active wheel pneumatic w/o small handrim	→ 2014-09-28	1
10	1508486	24" DHR active wheel solid w/o small handrim	→ 2014-09-28	1
11	1508484	Small handrim + fixations	→ 2014-09-28	1
12	1514223	24" Aluminium handrim + 6 fixations	→ 2014-09-28	1


Hand Control For Oad By Lever

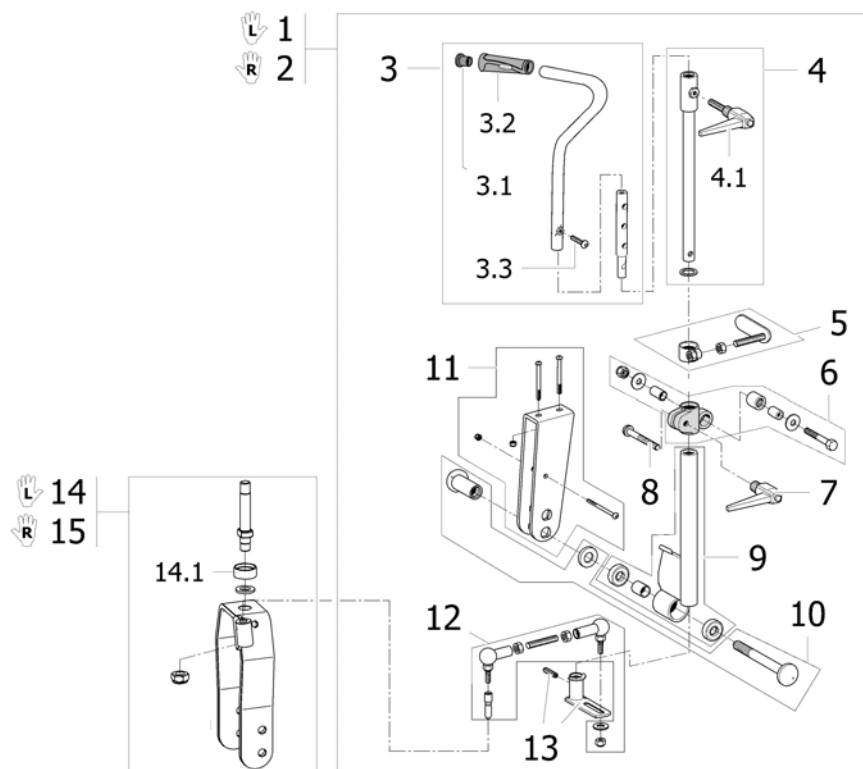

1535367


Pos.	Part number	Description	Valid from → until	Qty
1	1432936	Oad By Lever Hand Control Lh Completed Kit		1
2	1432935	Oad By Lever Hand Control Rh Completed Kit		1
3	1432624	Oad by lever hand control assy		1
3.1	1418112	Crossbrace back cap Ø22x15mm		1
3.2	1425473	Handle Invacare for backrest Ø22mm <i>Replace 1435421</i>		1
3.3	0029-40049	Cbhc screw M8x30; 8.8		1
4	1432625	Oad by lever hand lever shaft		1
4.1	0021-70002	Indexable lever 63mm M8x16		1
5	1432627	Oad by lever brake shoe assy		1
6	1432998	Oad by lever adjustable sleeve assy		1
7	0021-70011	Indexable lever 63 mm M8		1
8	0029-40252	Clamp screw m8 x 1 1/4"		1
9	1432999	Oad By Lever Equalizer		1
10	1433000	Equalizer Completed Axle		1
11	1433003	Equalizer Support + Fixation		1
12	1534241	Oad By Lever Ball Joint Kit		1
13	1433002	Oad by lever steering bush + fixation		1
14	1535369	Oad By Lever Fork And Axle Kit Lh		1
14.1	1529516-0618	Fork ring Champagne		1
15	1535368	Oad By Lever Fork And Axle Kit Rh		1

Oad By Lever

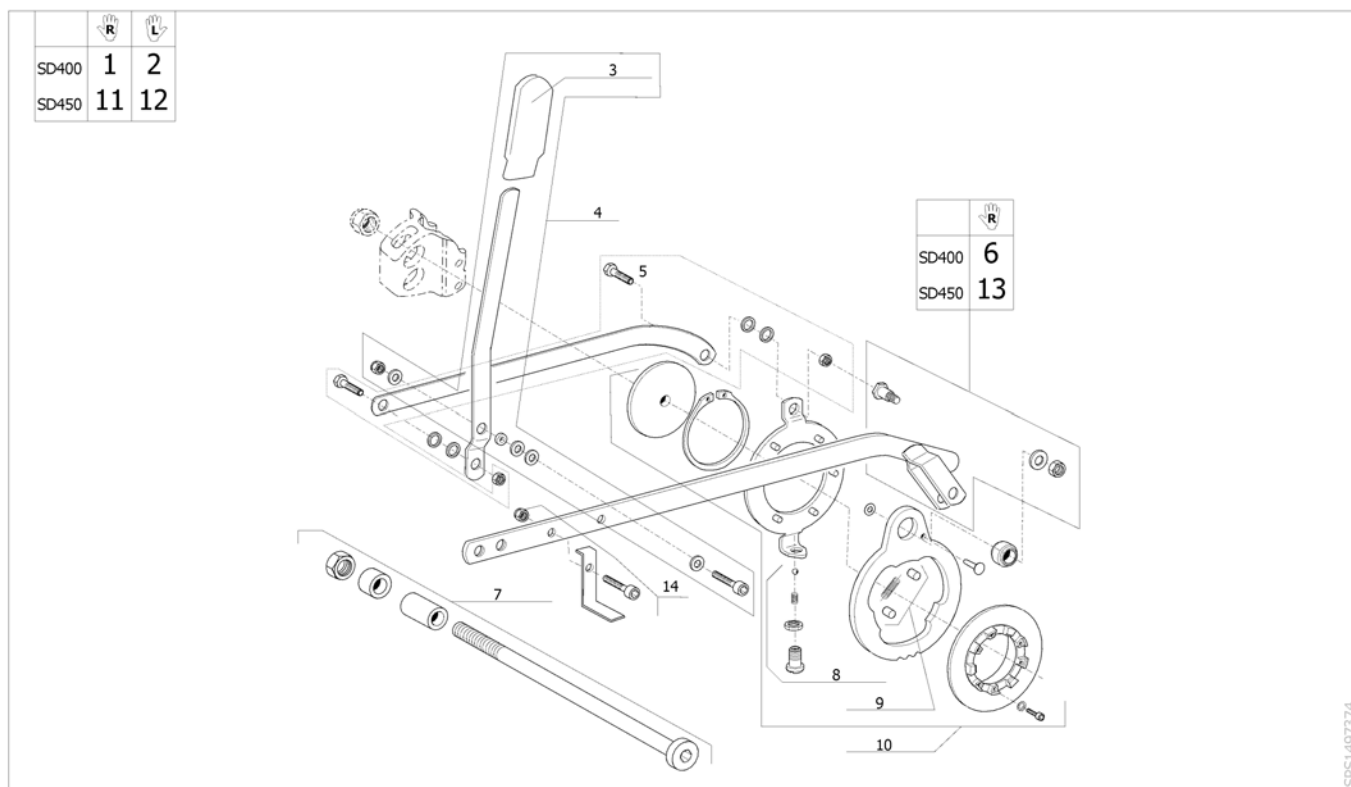

Pos.	Part number	Description	Valid from → until	Qty
1	1432932	Oad by lever Rh transmission kit SD400 mm		1
2	1432933	Oad by lever Lh transmission kit SD400 mm		1
3	0240-05576	End cap lever oad by lever		1
4	1432930	Oad by lever reverse link + fixation		1
5	1433010	Inversion link + fixation		1
6	1433006	Transmission link + fixation D400 mm		1
7	1496987	Oad By Lever Rear Wheel Axle Kit A4		1
8	9742-84304	Hollow screw + ball + spring		1
9	9028-40005	Oad by lever spring + cam		1
10	1433009	Driving cam completed		1
11	1435910	Oad By Lever Rh Transmission Kit SD450 mm		1
12	1435911	Oad By Lever Lh Transmission Kit SD450 mm		1
13	1435912	Transmission Link + Fixation D450 mm		1
14	1442034	Oad By Lever Brake Kit D450 mm		1


Wheels For Oad By Lever

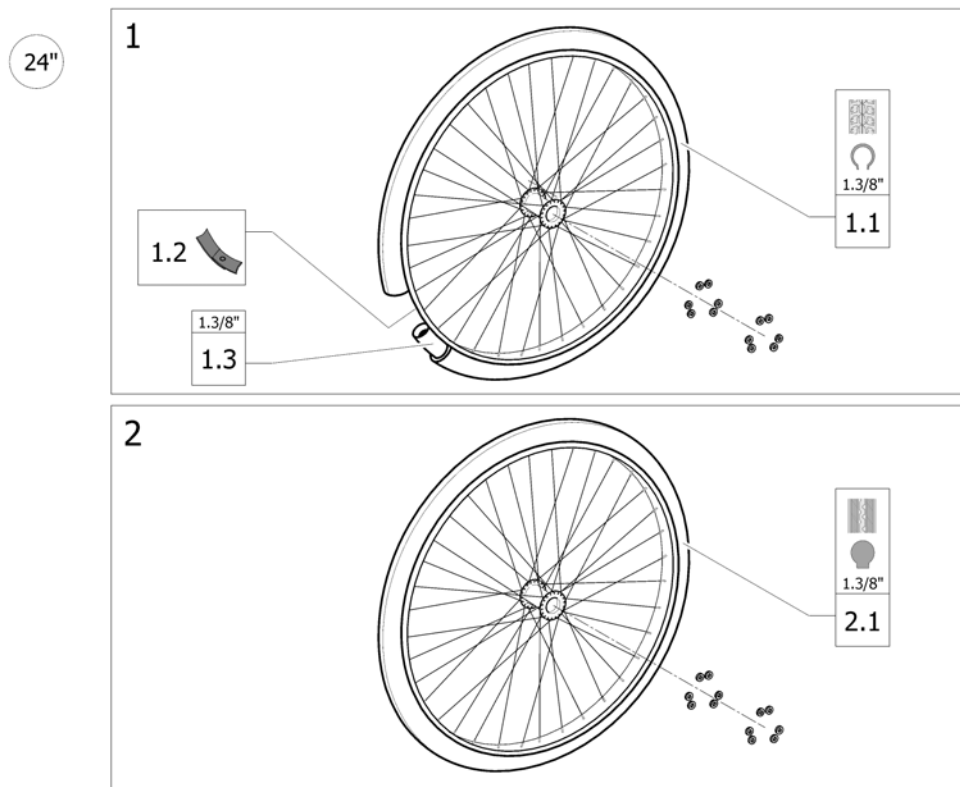

ISPS1507887


Pos.	Part number	Description	Valid from → until	Qty
1	1509380	24" x 1.3/8" oad spoked wheel + pneumatic tyre		2
1.1	0036-30001	Pneumatic tyre, 24" x 1.3/8", grey		2
1.2	0036-40003	Rim tape 24"		2
1.3	0036-20001	Inner tube, 24" x 1.3/8"		2
2	1509381	24" x 1.3/8" oad spoked wheel + solid tyre		2
2.1	0036-10013	Solid tyre, 24" x 1.3/8", grey		2

Rear wheel transit supports for manual brakes

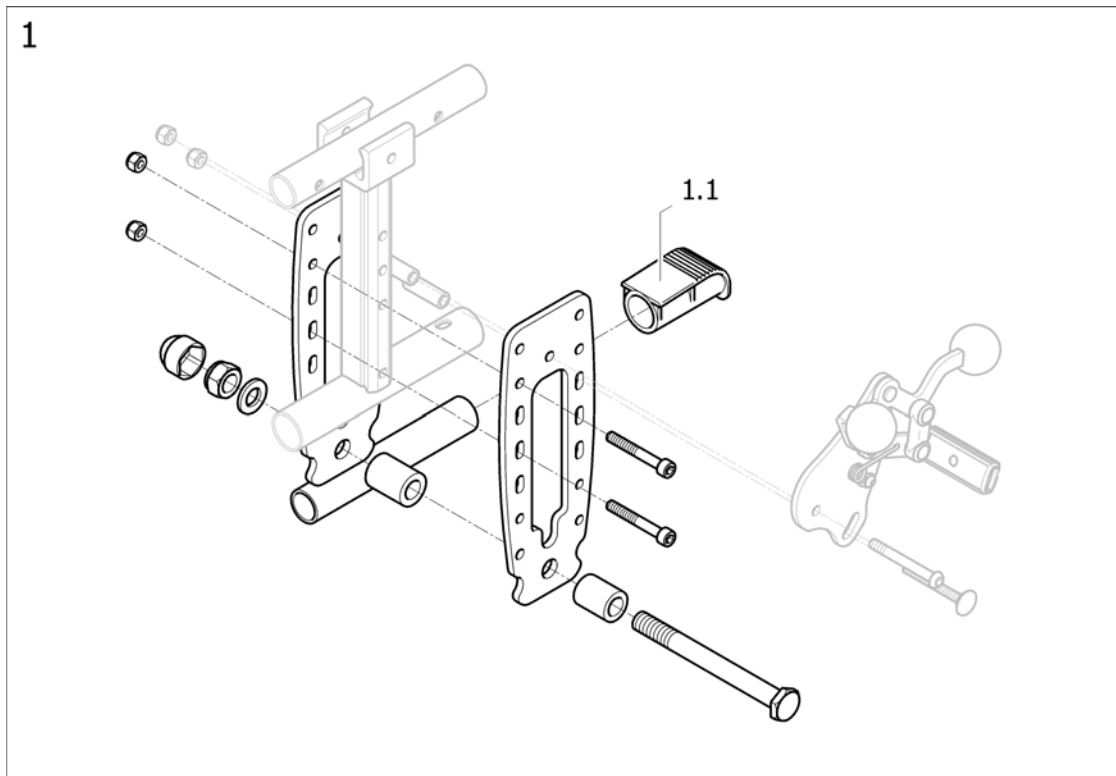


Pos.	Part number	Description	Valid from → until	Qty
1	1451675	Kit rear wheel support transit + axle for manual brake		2
1.1	0021-20011	Black end cap for tipping aid		2

Rear wheel transit supports for hub brakes

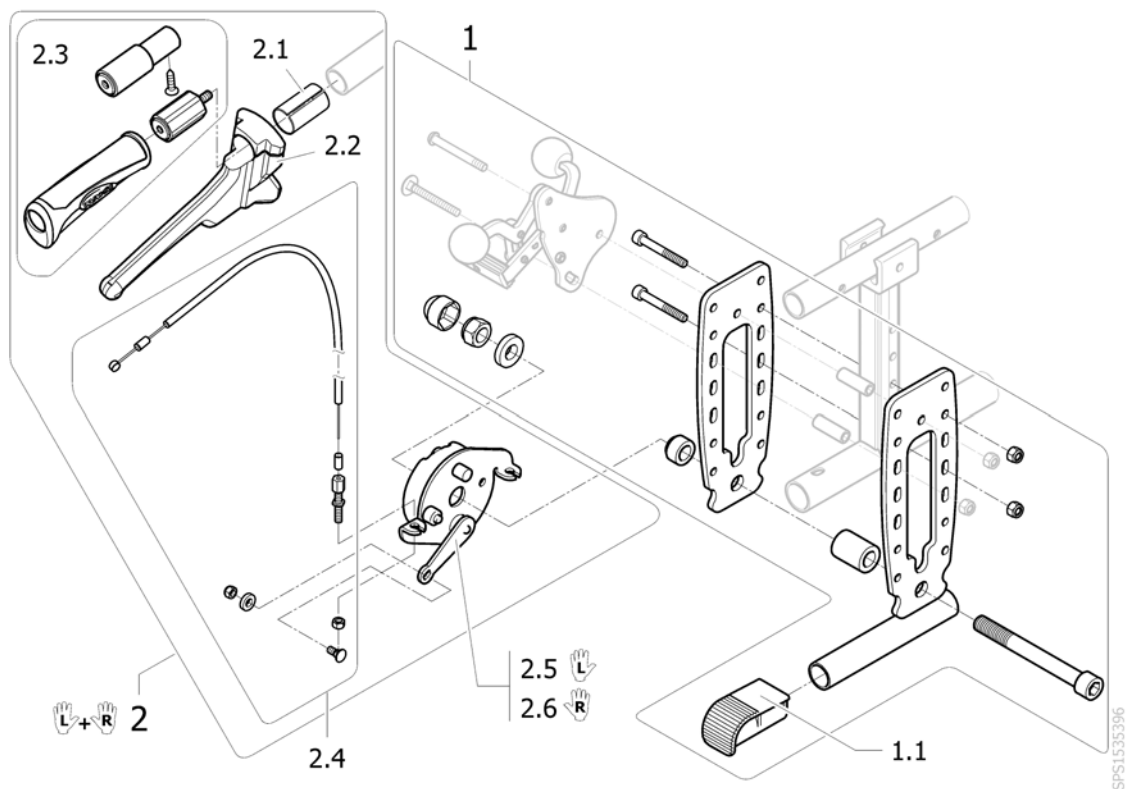

Pos.	Part number	Description	Valid from → until	Qty
1	1451676	Kit rear wheel support transit + axle for hub brake pneumatic tyre		2
1	1558350	Kit rear wheel support transit + axle for hub brake solid tyre		2
1.1	0021-20011	Black end cap for tipping aid		2
2	1567416	Kit pair of attendant hub brakes + tube extension + diameter adapter NG		1
2.1	5221363	Fixed Backrest Hub Brake Adaptation		2
2.2	1420527	Black Brake Lever dia 22mm		2
2.3	1430412	Back brace insert for fixed backrest tube		2
2.4	1431285	Attendant Hub Brakes Cable kit (Pair)		1
2.5	1558343	Lh hub brake pate		1
2.6	1558344	Rh hub brake plate		1
2.7	M30071	Cable Clamp Bolt c/wNut& Wash		2



Others Options & Accessories

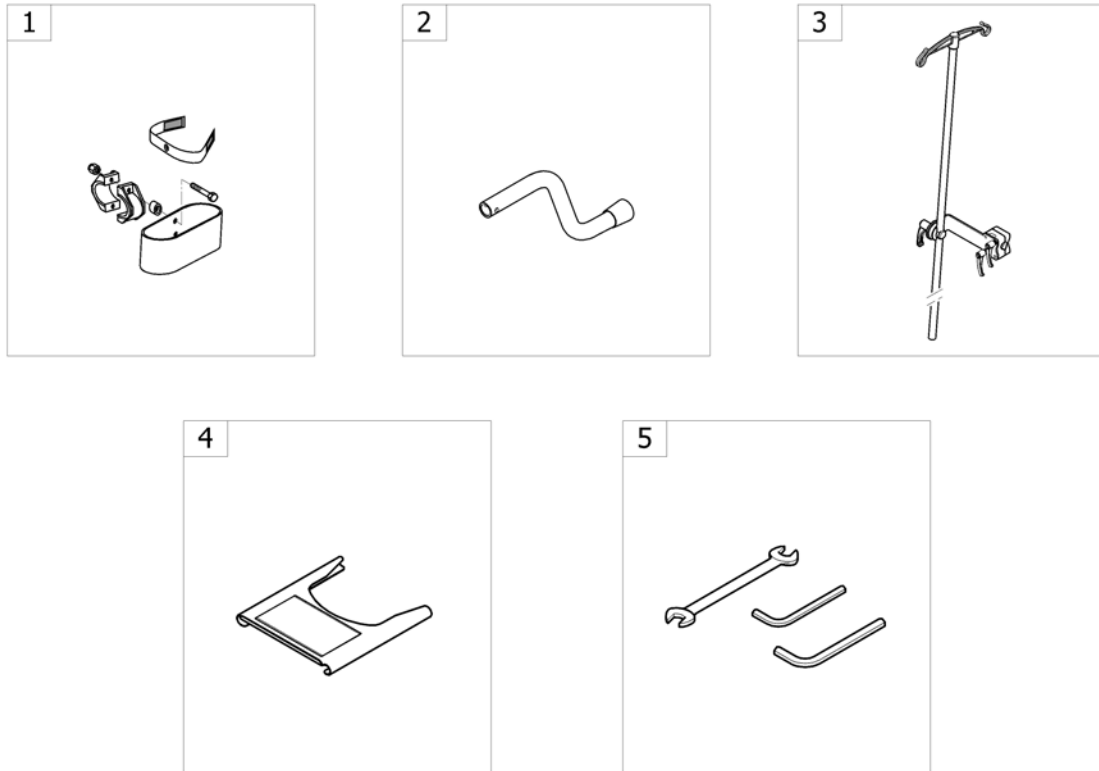

<SPCACTIONING98


Pos.	Part number	Description	Valid from → until	Qty
1	-	Cane holder		1
2	-	Tipping tube		1
3	-	Dripstand		1
4	-	Transparent tray		1
5	-	Kit Tools [CK_1251]		1

Cane holder

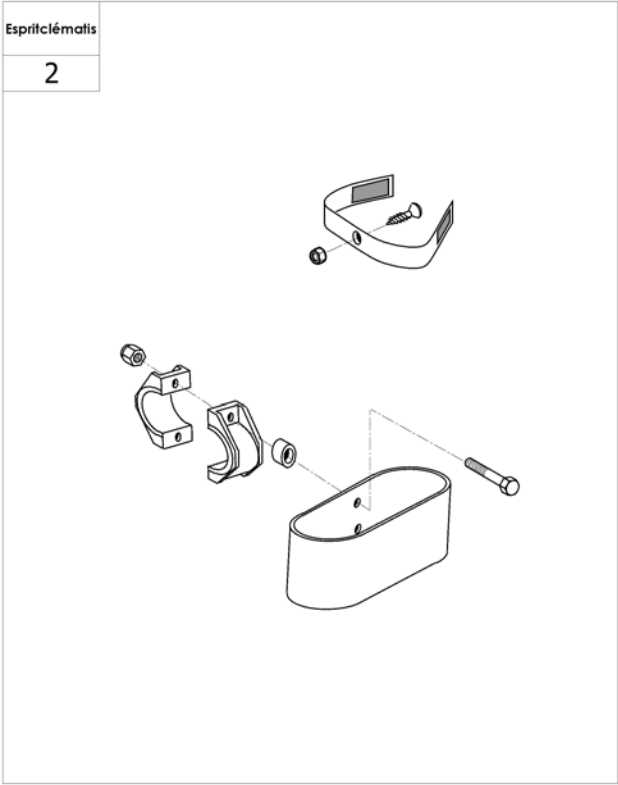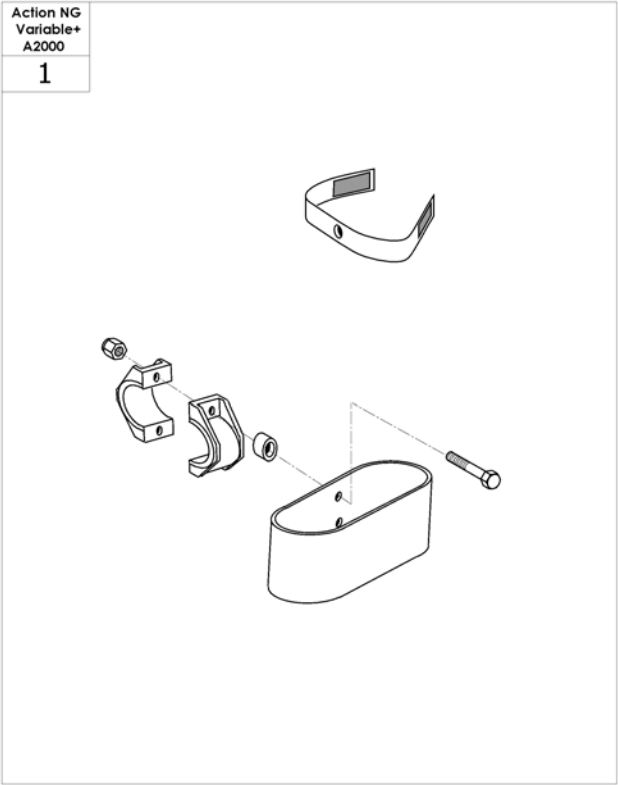

Pos.	Part number	Description	Valid from → until	Qty
1	0572-02056	Cane holder		1
2	1568838	Cane Holder Esprit Clem		1


Tipping tube

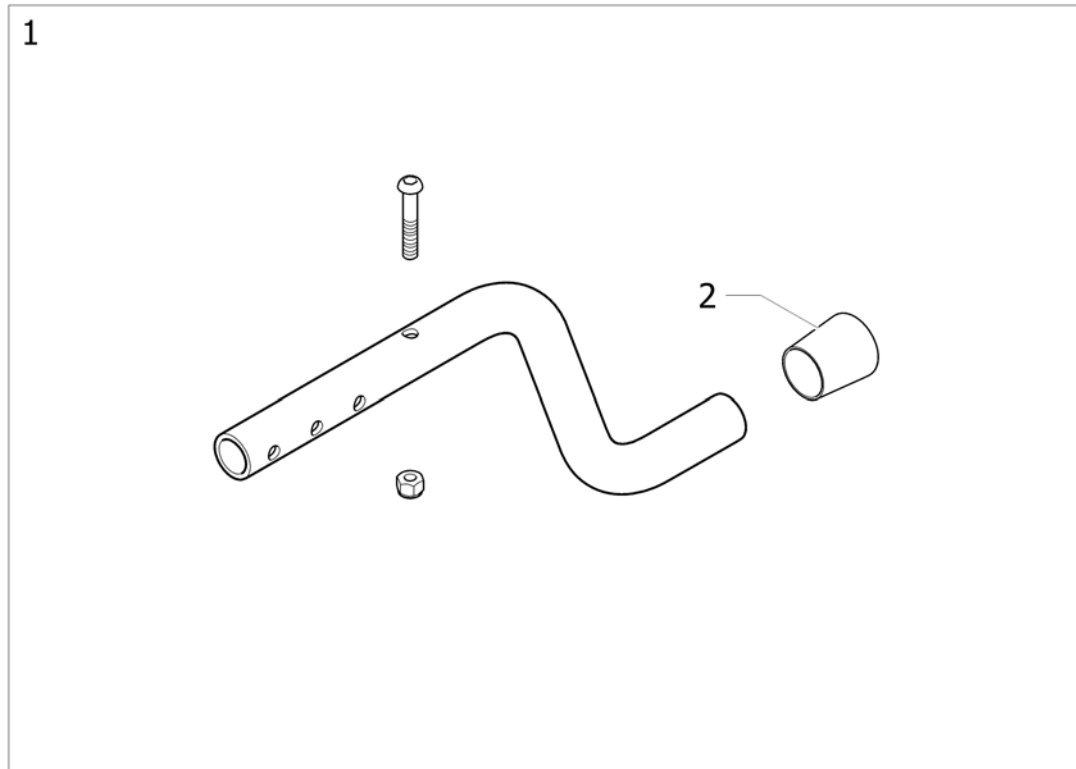

ISPS1552311 ~ sps1552311


Pos.	Part number	Description	Valid from → until	Qty
1	1548902	Tipping Tube		1
2	0021-30015	22mm Ext Cap		1

Dripstand

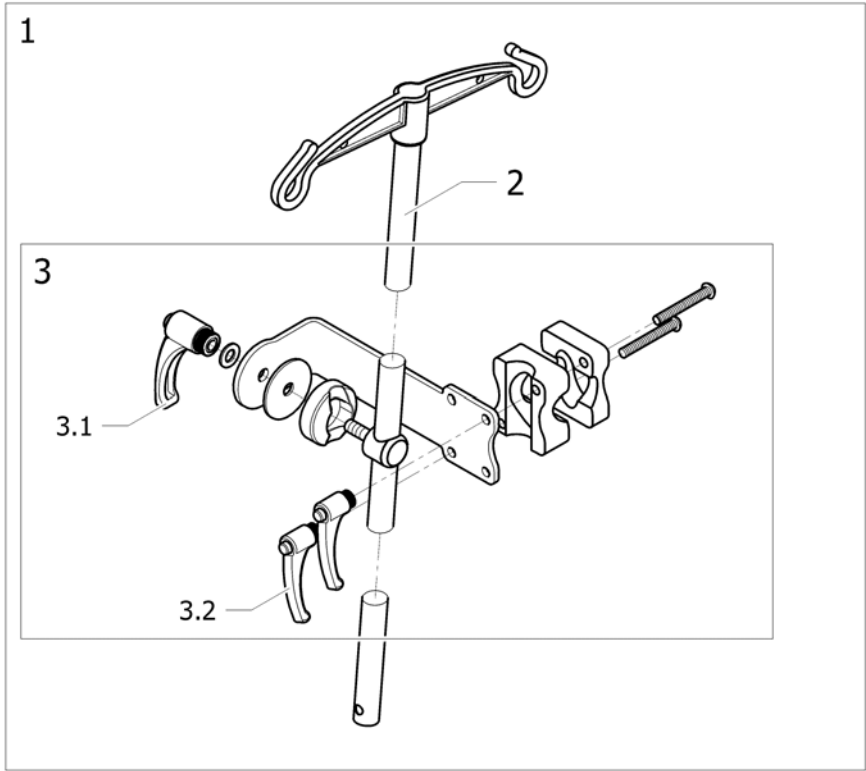


ISPS1535040

Pos.	Part number	Description	Valid from → until	Qty
1	1534212	Dripstand mounted on back brace		1
2	2240-06160	Dripstand		1
3	1541969	Drip stand support on backbrace		1
3.1	0021-70011	Indexable lever 63 mm M8		1
3.2	1428547	Indexable Lever; Female M6		2

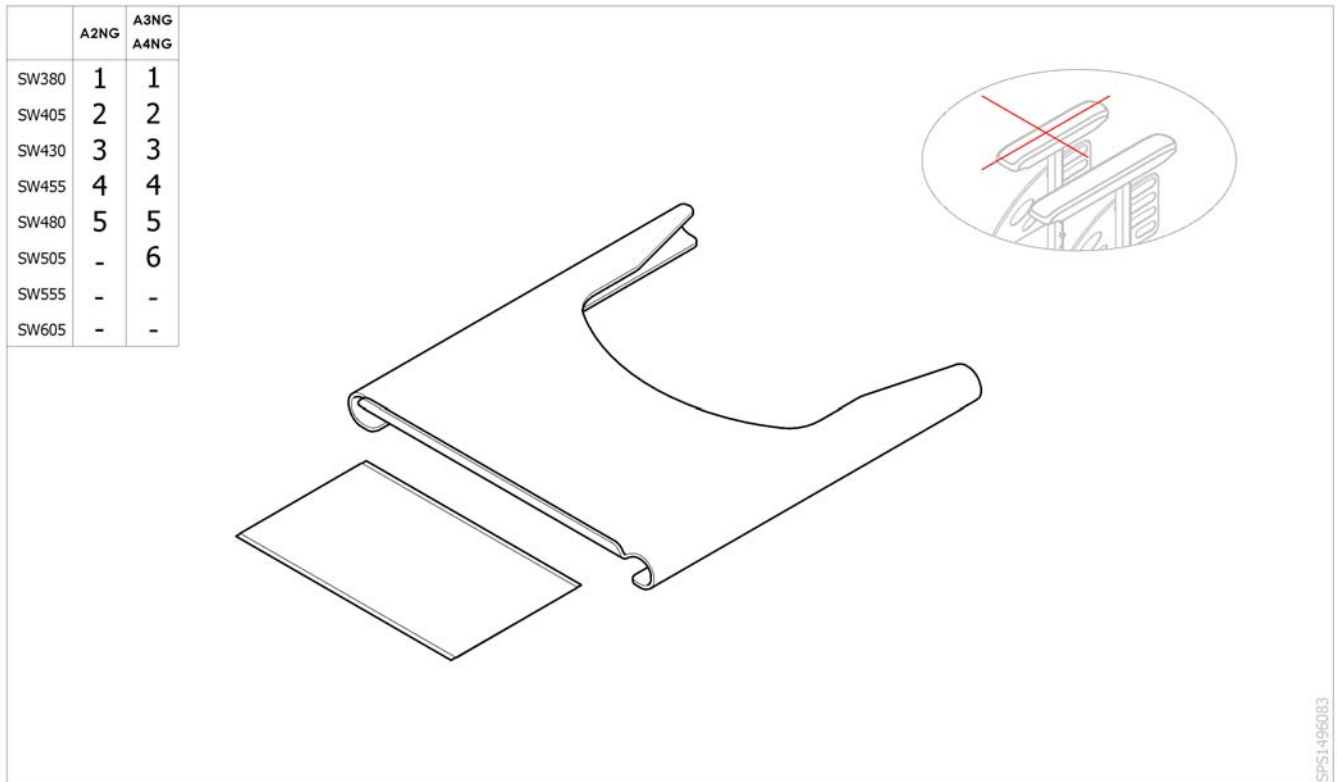Transparent tray

Pos.	Part number	Description	Valid from → until	Qty
1	1450921	Transparent tray Action with anti-slip mat SW380 mm		1
2	1450922	Transparent tray Action with anti-slip mat SW405 mm		1
3	1450923	Transparent tray Action with anti-slip mat SW430 mm		1
4	1450924	Transparent tray Action with anti-slip mat SW455 mm		1
5	1450925	Transparent tray Action with anti-slip mat SW480 mm		1
6	1450926	Transparent tray Action with anti-slip mat SW505 mm		1

Kit Tools

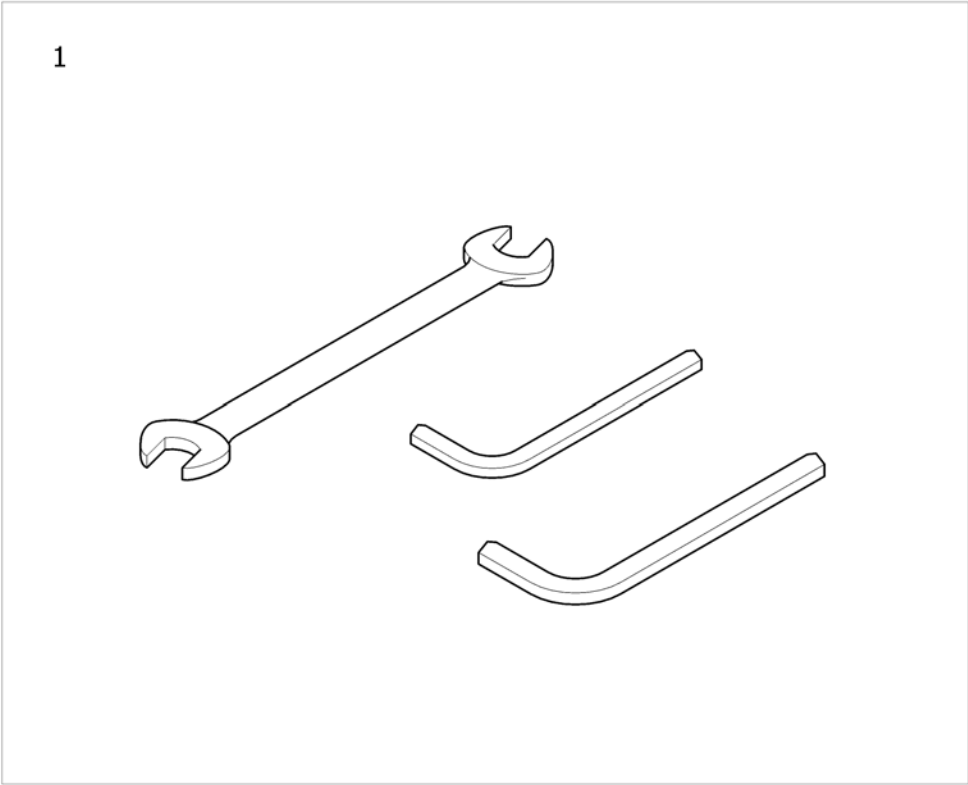

ISPS1602205

Pos.	Part number	Description	Valid from → until	Qty
1	SP1587482	Kit Tools		1



Yes, you can.®



Belgium & Luxemburg:

Invacare nv, Autobaan 22, B-8210 Loppem
Tel: (32) (0)50 83 10 10, Fax: (32) (0)50 83 10 11
belgium@invacare.com / www.invacare.be

Denmark:

Invacare A/S, Sdr. Ringvej 37, DK-2605 Brøndby
Tel: (45) (0)36 90 00 00, Fax: (45) (0)36 90 00 01
denmark@invacare.com / www.invacare.dk

Deutschland:

Invacare Aquatec GmbH, Alemannenstraße 10, D-88316 Isny
Tel: (49) (0)75 62 7 00 0, Fax: (49) (0)75 62 7 00 66
info@invacare-aquatec.com / www.invacare-aquatec.de

European Distributor Organisation:

Invacare, Kleiststraße 49, D-32457 Porta Westfalica
Tel: (49) (0)57 31 754 540, Fax: (49) (0)57 31 754 541
edo@invacare.com / www.invacare.eu.com

España:

Invacare SA, c/Areny s/n, Polígono Industrial de Celrà, E-17460 Celrà (Girona)
Tel: (34) (0)972 49 32 00, Fax: (34) (0)972 49 32 20
contactsp@invacare.com / www.invacare.es

France:

Invacare Poirier SAS, Route de St Roch, F-37230 Fondettes
Tel: (33) (0)2 47 62 64 66, Fax: (33) (0)2 47 42 12 24
contactfr@invacare.com / www.invacare.fr

Ireland:

Invacare Ireland Ltd, Unit 5 Seatown Business Campus, Seatown Road, Swords, County Dublin - Ireland
Tel: (353) 1 810 7084, Fax: (353) 1 810 7085
ireland@invacare.com / www.invacare.ie

Italia:

Invacare Mecc San s.r.l., Via dei Pini 62, I-36016 Thiene (VI)
Tel: (39) 0445 38 00 59, Fax: (39) 0445 38 00 34
italia@invacare.com / www.invacare.it

Nederland:

Invacare BV, Celsiusstraat 46, NL-6716 BZ Ede
Tel: (31) (0)318 695 757, Fax: (31) (0)318 695 758
nederland@invacare.com, csede@invacare.com / www.invacare.nl

Norge:

Invacare AS, Grensesvingen 9, Postboks 6230, Etterstad, N-0603 Oslo
Tel: (47) (0)22 57 95 00, Fax: (47) (0)22 57 95 01
norway@invacare.com, island@invacare.com / www.invacare.no

Österreich:

Invacare Austria GmbH, Herzog Odilostrasse 101, A-5310 Mondsee
Tel: (43) 6232 5535 0, Fax: (43) 6232 5535 4
info-austria@invacare.com / www.invacare.at

Portugal:

Invacare Lda, Rua Estrada Velha, 949, P-4465-784 Leça do Balio
Tel: (351) (0)225 1059 46/47, Fax: (351) (0)225 1057 39
portugal@invacare.com / www.invacare.pt

Sverige & Suomi:

Invacare AB, Fagerstagatan 9, S-163 91 Spånga
Tel: (46) (0)8 761 70 90, Fax: (46) (0)8 761 81 08
sweden@invacare.com, finland@invacare.com / www.invacare.se

Switzerland:

Invacare AG, Benkenstrasse 260, CH-4108 Witterswil
Tel: (41) (0)61 487 70 80, Fax: (41) (0)61 487 70 81
switzerland@invacare.com / www.invacare.ch

United Kingdom:

Invacare Limited, Pencoed Technology Park, Pencoed, Bridgend CF35 5AQ
Switchboard Tel: (44) (0)1656 776200, Fax: (44) (0)1656 776201
Customer services Tel: (44) (0)1656 776222, Fax: (44) (0)1656 776220
uk@invacare.com / www.invacare.co.uk