# Invacare® Dacapo®

Dacapo® Standard, Dacapo® Basic, Dacapo® Basic Low, Dacapo® Child, Dacapo® Square, Dacapo® Combi, Dacapo® Comfort, Dacapo® Comfort Plus, Dacapo® 2Sided, Dacapo® Heavy User, Dacapo® Heavy User Light, Dacapo® Premier, Dacapo® Kid, Dacapo® Top, Dacapo® Top Pro

<b>en</b>	<b>Mattress range</b>	
	<b>User Manual</b> .....	<b>3</b>
<b>da</b>	<b>Madrasserie</b>	
	<b>Brugsanvisning</b> .....	<b>19</b>
<b>no</b>	<b>Madrassområde</b>	
	<b>Bruksanvisning</b> .....	<b>35</b>
<b>sv</b>	<b>Madrassortiment</b>	
	<b>Bruksanvisning</b> .....	<b>51</b>
<b>fi</b>	<b>Patjavalikoima</b>	
	<b>Käyttöohje</b> .....	<b>67</b>

This manual **MUST** be given to the user of the product.  
**BEFORE** using this product, this manual **MUST** be read and saved for future reference.

**Yes, you can.®**

© 2020 Invacare Corporation

All rights reserved. Reproduction, duplication or modification in whole or in part is prohibited without prior written permission from Invacare. Trademarks are identified by ™ and ®. All trademarks are owned by or licensed to Invacare Corporation or its subsidiaries unless otherwise noted.

Making Life's Experiences Possible is a registered trademark in the U.S.A.

---

# Contents

---

This manual MUST be given to the user of the product.  
BEFORE using this product, this manual MUST be read and saved  
for future reference.

<b>1 General</b>	<b>4</b>
1.1 Introduction	4
1.2 Symbols in this Document	4
1.3 Compliance	4
1.4 Warranty Information	5
1.5 Limitation of Liability	5
1.6 Service Life	5
<b>2 Safety</b>	<b>6</b>
2.1 Safety Information	6
2.2 Safety Information for Transport	7
<b>3 Product Overview</b>	<b>8</b>
3.1 Intended Use	8
3.2 Indications	8
3.3 Symbols on the Product	8
3.4 Scope of Delivery	8
3.5 Mattresses	9
3.6 Mattress covers	10
<b>4 Usage</b>	<b>11</b>
4.1 Safety Information	11
4.2 Preparing the Mattress for Use	11
4.3 Using the handles	12
<b>5 Maintenance</b>	<b>13</b>
5.1 Inspection	13
5.2 Cleaning and Disinfection	13
5.2.1 General Safety Information	13
5.2.2 Cleaning Intervals	13
5.2.3 Cleaning Instructions	14
5.2.4 Disinfection Instructions	14

5.3 Replacing Cover	15
<b>6 After Use</b>	<b>16</b>
6.1 Storage	16
6.2 Reconditioning	16
6.3 Disposal	16
<b>7 Technical Data</b>	<b>17</b>
7.1 General Data	17
7.2 Materials	18

# 1 General

## 1.1 Introduction

This user manual contains important information about the handling of the product. To ensure safety when using the product, read the user manual carefully and follow the safety instructions.

Note that there may be sections in this document, which are not relevant to your product, since this document applies to all available models (on the date of printing). If not otherwise stated, each section in this document refers to all models of the product.

The models and configurations available in your country can be found in the country-specific sales documents.

Invacare reserves the right to alter product specifications without further notice.

Before reading this document, make sure you have the latest version. You find the latest version as a PDF on the Invacare website.

If you find that the font size in the printed document is difficult to read, you can download the PDF version from the website. The PDF can then be scaled on screen to a font size that is more comfortable for you.

For more information about the product, for example product safety notices and product recalls, contact your Invacare distributor. See addresses at the end of this document.

In case of a serious incident with the product, you should inform the manufacturer and the competent authority in your country.

## 1.2 Symbols in this Document

Symbols and signal words are used in this document and apply to hazards or unsafe practices which could result in personal injury or property damage. See the information below for definitions of the signal words.

### WARNING

Indicates a hazardous situation that could result in serious injury or death if it is not avoided.

### CAUTION

Indicates a hazardous situation that could result in minor or slight injury if it is not avoided.

### IMPORTANT

Indicates a hazardous situation that could result in damage to property if it is not avoided.

### Tips and Recommendations

Gives useful tips, recommendations and information for efficient, trouble-free use.

## 1.3 Compliance

Quality is fundamental to the company's operation, working within the disciplines of ISO 13485.

This product features the CE mark, in compliance with the Medical Device Regulation 2017/745 Class I. The launch date of this product is stated in the CE declaration of conformity.

We are continuously working towards ensuring that the company's impact on the environment, locally and globally, is reduced to a minimum.

We only use REACH compliant materials and components.

## **1.4 Warranty Information**

We provide a manufacturer's warranty for the product in accordance with our General Terms and Conditions of Business in the respective countries.

Warranty claims can only be made through the provider from whom the product was obtained.

## **1.5 Limitation of Liability**

Invacare accepts no liability for damage arising from:

- Non-compliance with the user manual
- Incorrect use
- Natural wear and tear
- Incorrect assembly or set-up by the purchaser or a third party
- Technical modifications
- Unauthorised modifications and/or use of unsuitable spare parts

## **1.6 Service Life**

The expected service life of this product is five years when used daily and in accordance with the safety instructions, maintenance intervals and correct use, stated in this manual. The effective service life can vary according to frequency and intensity of use.

## 2 Safety

### 2.1 Safety Information

Education, clinical judgement and action-based planning based on vulnerability are fundamental factors in prevention of pressure injuries.

A range of assessment scales can be used as a formal method of assessing risk from pressure injury development, and should be used in conjunction with an informal assessment (informed nursing judgement). Informal assessment is considered to be of greater importance and clinical value.

#### **WARNING!**

##### **Risk of Serious Injury or Damage**

Improper use of this product may cause injury or damage.

- If you are unable to understand the warnings, cautions or instructions, contact a health care professional or provider before attempting to use this equipment.
- Do not use this product or any available optional equipment without first completely reading and understanding these instructions and any additional instructional material such as user manual, service manual or instruction sheet supplied with this product or optional equipment.

#### **WARNING!**

##### **Risk of Developing Pressure Injuries**

The mattress delivers effective pressure reduction, when the support surface is covered with a cotton, cotton combination or linen bed sheet, and any one of these is the only item deployed between the support surface and the user.

- Bed sheets must be loosely fitted, with creases smoothed out.
- Ensure that the support surface in contact with the user is kept free from crumbs and other food debris, and that drip cables, stents, and other foreign objects do not become entrapped between the support surface and the user.
- Heated over blankets must only be used in consultation with a qualified healthcare professional, as an increase in temperature can increase the risk of developing pressure injuries.

#### **WARNING!**

##### **Risk of fire or explosion!**

A cigarette can burn a hole in the bed surface and cause damage to the mattress. Also, patient clothing, bed sheets, etc. may be combustible and cause a fire. Failure to observe this warning can result in a severe fire, property damage and cause physical injury or death.

- Take special care in oxygen rich environments.
- Do not smoke.

**CAUTION!****Risk of Injury**

Non-original or wrong parts may affect the function and safety of the product.

- Only use original parts for the product in use.
- Due to regional differences, refer to your local Invacare catalog or website for available options or contact your Invacare distributor. See addresses at the end of this document.

## 2.2 Safety Information for Transport

- Take care when handling mattresses to ensure no damage to the cover.
- It is recommended that two people lift/carry mattresses.
- Avoid contact with jewellery, nails, abrasive surfaces etc.
- Do not drag mattresses.
- Avoid contact with wall, door frames, door catches or locks etc.
- Do not transport in roll cages unless completely protected from the sharp edges of the cage.

### 3 Product Overview

#### 3.1 Intended Use

This mattress is intended to deliver effective pressure reduction and postural support to users. It is intended to be used in conjunction with an appropriately sized bed frame as part of an overall pressure injury prevention program of care.

#### 3.2 Indications

Suitable for users with a low risk of developing pressure injuries and users who require a comfortable mattress.

Dacapo Combi, Comfort Plus, Square, 2Sided, Premier, Heavy User and Heavy User Light are pressure injury-preventive and can be used in cases of low to medium risk of developing pressure injuries and for the care of pressure injuries at stage 1 or 2.

#### Contraindications

Not suitable for users with more severe pressure injuries.

#### 3.3 Symbols on the Product


	Medical Device		European conformity
	Manufacturer		Date of manufacture

	European representative		Read user manual
	Do not pierce or cut		Line dry
	Tumble dry low heat		Machine wash temperature max. 95 °C
	Do not tumble (Cotton cover)		
	User weight limit. See 7 <i>Technical Data</i> , page 17.		Do not iron
	Do not put near flame		Do not bleach
	Dry clean, PCE only		LOT number

#### 3.4 Scope of Delivery

The following components are included within the scope of delivery: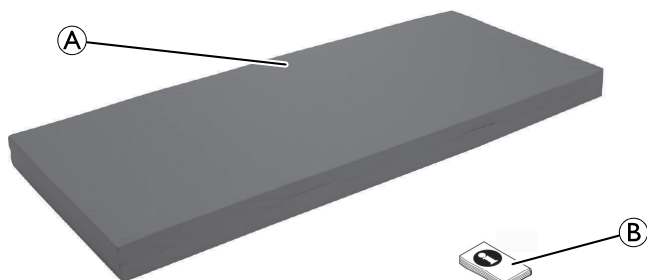

A	Mattress with cover
B	User Manual

### 3.5 Mattresses

The mattresses offer pressure reducing qualities, specific to the individual's needs. By promoting even weight distribution, they reduce tissue pressure at potentially vulnerable areas.

#### Dacapo Standard

The mattress satisfies basic requirements for good cushioning. The high density, high resilience foam provides a firm supportive base.

#### Dacapo Basic

The mattress fully satisfies basic comfort and hygiene requirements for a sleeping surface. The high density, high resilience foam provides a firm supportive base.

#### Dacapo Basic Low

The high density, high resilience foam provides a firm supportive base.

#### Dacapo Child

The foam core is made from environmentally friendly organic foam. The high density of the foam allows for a good pressure distribution.

#### Dacapo Square

The mattress is constructed from a top layer of environmentally friendly organic foam and a base of high resilience foam. The unique square patterned body contour cut into the top layer, minimises surface pressure, which allows pressure relief and body pressure equalisation.

#### Dacapo Combi

The mattress is constructed from a top layer of visco-elastic foam and is combined with a base of high resilience foam. The top foam is a heat-sensitive material which works in conjunction with body temperature and movement, while the base of high resilience foam ensures strength and robust quality.

#### Dacapo Comfort / Comfort Plus

The foam core is made from environmentally friendly organic foam. The high density of the foam allows for a good pressure distribution. The upper surface has been cut into squares, all being individually flexible.

#### Dacapo 2Sided

The mattress is constructed with 2 foam layers.

One side is made from environmentally friendly organic foam. The high density of the foam allows for a good pressure distribution. The upper surface has been cut into squares, all being individually flexible.

The other side is composed of visco-elastic foam which adapts to the user's body dynamics and temperature.

### **Dacapo Heavy User / Heavy User Light**

The mattress is constructed of three layers to offer the user firm responsive support, pressure relief and cushioning. The lower part of the mattress is constructed from a hard high resilience foam with a high density. The middle layer is a softer version of the high resilience foam and the upper layer of the mattress is visco-elastic foam.

### **Dacapo Premier**

The mattress features a unique contoured insert pad which provides patient comfort and optimises pressure reduction. The insert is supported by a foam u-core that facilitates patient transfer by providing effective side-wall support.

### **Dacapo Kid**

The mattress is constructed from a single layer of high resilience foam. The unique square-patterned body contour cut into the mattress, minimises surface pressure, which allows pressure redistribution.

### **Dacapo Top**

This is an overlay designed to be placed on top of basic mattresses. The overlay is composed of visco-elastic foam which adapts to the user's body dynamics and temperature.

### **Dacapo Top Pro**

This is an overlay designed to be placed on top of basic mattresses. The overlay is composed of a castellated high resilience foam which provides a firm supportive base.

## **3.6 Mattress covers**

The Dacapo range has three cover options available:

### **Wipedown cover**

The wipedown cover is composed of polyurethane coated polyester. The cover is two-way stretch and gentle on the skin. The fabric is vapour-permeable and water-resistant.

This cover is allergy-tested and treated against dust mites.

### **Evacuation cover**

The evacuation is composed of two-way stretch, vapour-permeable, water-resistant, polyurethane coated polyester. These covers are equipped with handles on all four sides. On the two long sides there are straps, which in case of an emergency evacuation can be wrapped around the user.

This cover is allergy-tested and treated against dust mites.

### **Cotton cover**

The cotton cover is composed of standard cotton, washable at 40 °C.

## 4 Usage

### 4.1 Safety Information

#### **WARNING!**

Insufficient repositioning of the patient can lead to tissue compression and potential injury formation.

To relieve pressure, it is very important for the patients to reposition themselves, or to be repositioned, on a regular basis.

- Before using the product, always consult a qualified healthcare professional for clinical judgement.
- Monitor the patient frequently.

#### **CAUTION!**

- Make sure that the printed side of the mattress cover always faces upwards.
- Ensure that the distance between the surface of the mattress and the top of the side rail is at least 220 mm.

#### **CAUTION!**

##### **Risk of damage to the mattress**

If holes are present in the mattress cover, there is a risk that liquids may ingress and contamination may occur.

- Ensure that the mattress is not jammed or damaged by sharp edges.
- Do not place hypodermic needles, venflons, scalpels or other similarly sharp objects on or under the mattress.
- Do not use electrically heated blankets directly on or under the mattress.
- Ensure that all venflons are taped down correctly with no sharp edges exposed.
- When using bridging boards or other patient transfer aids, check for any sharp edges or burrs before use.
- When using the mattress on a profiling bed ensure that the knee break is used before the backrest.
- Attach medical equipment including infusion pumps and monitors to appropriate bed accessories.
- Avoid cigarette burns and pet claws puncturing the mattress cover.

### 4.2 Preparing the Mattress for Use

#### **CAUTION!**

##### **Risk of Injury or Damage to Property**

- Before use, check the product for any defect.
- Do not use a defective product.

1. Remove all packaging before use.
2. Place the mattress directly on the mattress support of the bed or place the mattress overlay on the existing mattress.

Compress rolled mattresses must be left to recover before use. Follow the instructions on the packaging inlay.

Mattresses that are delivered flat packed can be used immediately.

### 4.3 Using the handles

#### Handles on the Wipedown cover / Cotton cover

#### **CAUTION!**

#### **Risk of injury**

The handles on the cover are meant to transport the mattress only.

- Do not use the handles to move patients on the mattress.

#### Handles on the Evacuation cover

The Evacuation cover is equipped with handles on all four sides. On the two long sides are straps, which can be wrapped around the patient.

When the straps are not in use, they can be tucked away under the hook-and-loop pads. The handles can be used for lifting/dragging the mattress.

1. In case of an emergency evacuation, wrap the straps on the long sides around the patient.
2. Make sure that the arms and legs are within the straps, and the straps are tight so that the mattress will bend up around the patient.

## 5 Maintenance

### 5.1 Inspection

Check mattresses (foam and cover) for strike-through (this may include fluid ingress, stains, rips or damage) after the release of each patient, after ending of the period of use or on a minimum monthly basis (depending on which occurs first) by a suitably qualified and competent person.

#### Check mattress

1. Unzip the cover completely.
2. Check for any staining on the white underside of the cover.
3. Check for any staining on the interior foam.
4. Replace any stained items and dispose of as per local authority procedure.

### 5.2 Cleaning and Disinfection

#### 5.2.1 General Safety Information

#### CAUTION!

##### Risk of Contamination

- Take precautions for yourself and use appropriate protective equipment.

#### IMPORTANT!

- Wrong fluids or methods can harm or damage the product.
- All cleaning agents and disinfectants used must be effective, compatible with one another and must protect the materials they are used to clean.
  - Never use corrosive fluids (alkalines, acid etc.), abrasive cleaning agents or solvents (cellulose thinner, acetone etc.). We recommend a mild detergent.
  - Always make sure that the product is completely dried before taking it into use again.

For cleaning and disinfection in clinical or long-term care environments, follow your in-house procedures.

#### 5.2.2 Cleaning Intervals

#### IMPORTANT!

- Regular cleaning and disinfection enhances smooth operation, increases the service life and prevents contamination.
- Clean and disinfect the product
- regularly while it is in use,
  - before and after any service procedure,
  - when it has been in contact with any body fluids,
  - before using it for a new user.

**!** **IMPORTANT!**

- Keep a cleaning record as part of cleaning the system.

### 5.2.3 Cleaning Instructions

#### Cleaning Covers

(Removal of contaminants such as dust and organic matter)

1. Remove all covers for laundering.
2. Launder the covers with the recommended temperature at 95 °C (Cotton cover 40 °C) using a diluted detergent solution (Instructions on label).

**!** **IMPORTANT!**

- Washing at higher temperatures will cause shrinkage.

#### Drying Covers

1. Hang covers from a line or bar and drip dry in a clean indoor environment.  
or  
Tumble dry on a low heat setting.

**!** **IMPORTANT!**

- Tumble dry setting must not exceed 40 °C.
- Do not tumble dry for longer than 10 minutes.
- Dry thoroughly before use.

### 5.2.4 Disinfection Instructions

**!** **IMPORTANT!**

- Only use disinfectants and methods approved by your local infection control institution and follow your local infection control policy.
- Follow your local decontamination protocols.

#### Disinfecting covers

(Reducing the number of microorganisms)

**!** **IMPORTANT!**

- Failure in disinfection process may result in the accumulation of reagent that could damage the polyurethane coating, react with the bed frame, or negate the biocompatibility results.
  - Ensure that all cleaning agents, and disinfectants, are thoroughly rinsed off.
  - Dry thoroughly before use.

1. Wipe down the cover with a suitable detergent.
2. Rinse the cover thoroughly with clean water using a single use nonabrasive cloth.
3. Dry the cover thoroughly.

**Heavy soilage**

Where the mattress is badly soiled, we recommend cleaning with a dilute cleaning solution at maximum temperature in the washing machine (see product label).

- Clean up all spillages of bodily fluids i.e. blood, urine, faeces, sputum, wound exudate and all other bodily secretions as soon as possible using a suitable detergent.
- Large spillages of blood should be absorbed and removed with paper towels first.

**IMPORTANT!**

- Ensure that the corners of the foam insert are positioned correctly into the corners of the cover.
- Ensure that the castellated foam is facing upwards when inserted into its cover.

**WARNING!**

- In event of contamination, contact your hygiene specialist.
- Remove contaminated foams from use.

**WARNING!**

- In event of contamination, contact your hygiene specialist.
- Remove contaminated foams from use.

**Autoclaving covers**

Autoclave the cover at 110° C.

**IMPORTANT!**

- Make sure the cover is outstretched (not folded).
- Do not place covers on top of each other.

**5.3 Replacing Cover**

1. Unzip cover and remove it carefully from foam insert.
2. Place new cover onto foam insert.
3. Close zipper.

## 6 After Use

### 6.1 Storage

#### **IMPORTANT!**

- Store mattresses in a dry environment.
- Store mattresses within a protective cover.
- Store items on clean, dry (off-floor) surface with no sharp edges to avoid any possible damage.
- Never store other items on top of a mattress.
- Do not store mattresses next to radiators or other heating devices.
- Protect mattresses from direct sunlight.

For environmental conditions for storage, see *7 Technical Data, page 17*.

### 6.2 Reconditioning

This product is suitable for reuse. To recondition the product for a new user, carry out the following actions:

- Inspection
- Cleaning and disinfection

For detailed information, see *5 Maintenance, page 13*.

Make sure that the user manual is handed over with the product.

If any damage or malfunction is detected, do not reuse the product.

### 6.3 Disposal

Be environmentally responsible and recycle this product through your recycling facility at its end of life.

Disassemble the product and its components, so the different materials can be separated and recycled individually.

The disposal and recycling of used products and packaging must comply with the laws and regulations for waste handling in each country. Contact your local waste management company for information.

## 7 Technical Data

### 7.1 General Data

The mattresses are available in various sizes, depending on the market. Contact your Invacare distributor for detailed information.

Local requirements / certification available upon request from Invacare distributor.

Product	Dimensions [mm]			Maximum user weight [kg]	Weight of product [kg] <sup>1</sup>
	Width	Length	Height		
Dacapo Standard	830 — 1200	2000	50, 100	100	7 – 10.5
Dacapo Basic	700 — 900	1800 – 2200	120	130	8.5 – 12
Dacapo Basic Low	830, 880, 900	2000	50, 70	130	3.5 – 5
Dacapo Heavy User	900, 1160	2050, 2100	150	350	12 – 17.5
Dacapo Heavy User Light	800 — 1200	2000 — 2200	150	250	11 – 16.5
Dacapo Child	540 — 800	1370 — 1800	80	100	3
Dacapo Square	800 — 1200	1800 — 2200	140	140	10 – 15
Dacapo Combi	800 — 1180	2000 — 2200	140	140	8.5 – 13
Dacapo Comfort	800 — 1200	2000 — 2200	120	130	11.5 – 17
Dacapo Comfort Plus	800 — 1200	2000 — 2200	140, 150	140	13.5 – 19
Dacapo 2Sided	800, 850, 900	2000, 2200	150	130	11.5 – 17
Dacapo Premier	800 — 1200	2000, 2150, 2200	150	180	14
Dacapo Kid	800	1380 — 1770	100	70	5.0 – 6.5

1. Weight can vary depending on mattress size. Average weight used as an indication.

Product	Dimensions [mm]			Maximum user weight [kg]	Weight of product [kg] <sup>1</sup>
	Width	Length	Height		
Dacapo Top	750 — 1200	1700 — 2200	70	130	5 – 7.5
Dacapo Top Pro	800 — 1200	2000, 2150, 2200	70	150	4.5 – 5

## 7.2 Materials

Foam	High Resilience Polyurethane Foam
Whippedown cover / Evacuation cover	Polyurethane coated polyester knitted fabric
Cotton cover	100 % cotton fabric

All product components do not contain natural rubber latex.

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Invacare Limited
Invacare UK Operations Limited