# Invacare® Essential

Essential Care, Essential Basic, Essential Visco, Essential Plus, Essential Wave, Essential Touch

en	Mattresses	
	User Manual .....	2
da	Madrasser	
	Brugsanvisning .....	6
de	Matratzen	
	Gebrauchsanweisung .....	10
es	Colchones	
	Manual del usuario .....	14
it	Materassi	
	Manuale d'uso .....	19
nl	Matrassen	
	Gebruiksaanwijzing .....	23
no	Madrasser	
	Bruksanvisning .....	28
pt	Colchões	
	Manual de utilização .....	32
sv	Madrasser	
	Bruksanvisning .....	37

This manual MUST be given to the user of the product.  
BEFORE using this product, this manual MUST be read and saved for future reference.

**Yes, you can.®**

## 1 General

### 1.1 Introduction

This user manual contains important information about the handling of the product. To ensure safety when using the product, read the user manual carefully and follow the safety instructions.

Invacare reserves the right to alter product specifications without further notice.

Before reading this document, make sure you have the latest version. You find the latest version as a PDF on the Invacare website.

If you find that the font size in the printed document is difficult to read, you can download the PDF version from the website. The PDF can then be scaled on screen to a font size that is more comfortable for you.

In case of a serious incident with the product, you should inform the manufacturer and the competent authority in your country.

### 1.2 Symbols in this Document

Symbols and signal words are used in this document and apply to hazards or unsafe practices which could result in personal injury or property damage. See the information below for definitions of the signal words.

#### WARNING

Indicates a hazardous situation that could result in serious injury or death if it is not avoided.

#### CAUTION

Indicates a hazardous situation that could result in minor or slight injury if it is not avoided.

#### IMPORTANT

Indicates a hazardous situation that could result in damage to property if it is not avoided.

#### Tips and Recommendations

Gives useful tips, recommendations and information for efficient, trouble-free use.

### 1.3 Compliance

Quality is fundamental to the company's operation, working within the disciplines of ISO 13485.

This product features the CE mark, in compliance with the Medical Device Regulation 2017/745 Class I. The launch date of this product is stated in the CE declaration of conformity.

We are continuously working towards ensuring that the company's impact on the environment, locally and globally, is reduced to a minimum.

We only use REACH compliant materials and components.

### 1.4 Warranty Information

We provide a manufacturer's warranty for the product in accordance with our General Terms and Conditions of Business in the respective countries.

Warranty claims can only be made through the provider from whom the product was obtained.

### 1.5 Limitation of Liability

Invacare accepts no liability for damage arising from:

- Non-compliance with the user manual
- Incorrect use

- Natural wear and tear
- Incorrect assembly or set-up by the purchaser or a third party
- Technical modifications
- Unauthorised modifications and/or use of unsuitable spare parts

### 1.6 Service Life

The expected service life of this product is five years when used daily and in accordance with the safety instructions, maintenance intervals and correct use, stated in this manual. The effective service life can vary according to frequency and intensity of use.

## 2 Safety

### 2.1 Safety Information

Education, clinical judgement and action-based planning based on vulnerability are fundamental factors in prevention of pressure injuries.

A range of assessment scales can be used as a formal method of assessing risk from pressure injury development, and should be used in conjunction with an informal assessment (informed nursing judgement). Informal assessment is considered to be of greater importance and clinical value.

#### WARNING!

##### Risk of Serious Injury or Damage

Improper use of this product may cause injury or damage.

- If you are unable to understand the warnings, cautions or instructions, contact a health care professional or provider before attempting to use this equipment.
- Do not use this product or any available optional equipment without first completely reading and understanding these instructions and any additional instructional material such as user manual, service manual or instruction sheet supplied with this product or optional equipment.

#### WARNING!

##### Risk of Developing Pressure Injuries

The device delivers effective pressure reduction, when the support surface is covered with a cotton, cotton combination or linen bed sheet, and any one of these is the only item deployed between the support surface and the user.

- Bed sheets must be loosely fitted, with creases smoothed out.
- Ensure that the support surface in contact with the user is kept free from crumbs and other food debris, and that drip cables, stents, and other foreign objects do not become entrapped between the support surface and the user.
- Heated over blankets must only be used in consultation with a qualified healthcare professional, as an increase in temperature can increase the risk of developing pressure injuries.

**WARNING!****Risk of fire or explosion!**

A cigarette can burn a hole in the bed surface and cause damage to the device. Also, patient clothing, bed sheets, etc. may be combustible and cause a fire. Failure to observe this warning can result in a severe fire, property damage and cause physical injury or death.

- Take special care in oxygen rich environments.
- Do not smoke.

**CAUTION!****Risk of Injury**

Non-original or wrong parts may affect the function and safety of the product.

- Only use original parts for the product in use.
- Due to regional differences, refer to your local Invacare catalog or website for available options or contact your Invacare distributor. See addresses at the end of this document.

**2.2 Safety Information for Transport**

- Take care when handling the product to ensure no damage to the cover.
- It is recommended that two people lift/carry the product.
- Avoid contact with jewellery, nails, abrasive surfaces etc.
- Do not drag the product.
- Avoid contact with wall, door frames, door catches or locks etc.
- Do not transport in roll cages unless completely protected from the sharp edges of the cage.

**2.3 Symbols on the Product**

	Medical Device		European conformity
	Manufacturer		Date of manufacture
	European representative		Read user manual
	Do not pierce or cut		Line dry
	Do not dry clean		Machine wash temperature. Max. temperature is shown in symbol.
	User weight limit. See 7 Technical Data, page 5.		Do not iron
	Do not put near flame		Tumble dry low heat
	Do not bleach		LOT number

**3 Product Overview****3.1 Product Description**

The mattress offers pressure reducing qualities, specific to the individuals needs. By promoting even weight distribution, they reduce tissue pressure at potentially vulnerable areas.

The mattress is zipped around 3 sides for ease of inspection and cleaning.

The water-resistant cover provides a vapour-permeable, multi stretch surface, to promote patient comfort and to maximise the effectiveness of the foam core.

Essential Touch: Double sided castellated foam allows the mattress to be turned or rotated inside the cover for increased product longevity.

**3.2 Intended Use**

This mattress is intended to deliver effective pressure redistribution to users. It is intended to be used in conjunction with an appropriately sized bed frame as part of an overall pressure ulcer prevention program of care.


**Indications for Use**

Essential Basic: Suitable for users who do not require therapy for pressure ulcer.

Essential Care, Visco, Plus, Wave, Touch: Suitable for users 'At Risk/ High Risk' of developing pressure ulcers.

**3.3 Components**

The following components are included within the scope of delivery:

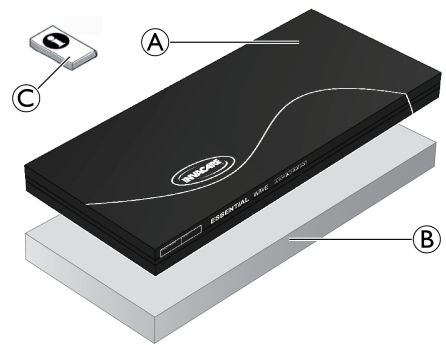(A)	Multi-stretch waterproof polyurethane cover
(B)	Foam insert
(C)	User Manual

**4 Usage****4.1 Safety Information****WARNING!**

Insufficient repositioning of the patient can lead to tissue compression and potential injury formation. To relieve pressure, it is very important for the patients to reposition themselves, or to be repositioned, on a regular basis.

- Before using the product, always consult a qualified healthcare professional for clinical judgement.
- Monitor the patient frequently.

**CAUTION!**

- Make sure that the printed side of the mattress cover always faces upwards.
- Ensure that the distance between the surface of the mattress and the top of the side rail is at least 220 mm.

**CAUTION!****Risk of damage to the mattress**

- If holes are present in the mattress cover, there is a risk that liquids may ingress and contamination may occur.
- Ensure that the mattress is not jammed or damaged by sharp edges.
  - Do not place hypodermic needles, venflons, scalpels or other similarly sharp objects on or under the mattress.
  - Do not use electrically heated blankets directly on or under the mattress.
  - Ensure that all venflons are taped down correctly with no sharp edges exposed.
  - When using bridging boards or other patient transfer aids, check for any sharp edges or burrs before use.
  - When using the mattress on a profiling bed ensure that the knee break is used before the backrest.
  - Attach medical equipment including infusion pumps and monitors to appropriate bed accessories.
  - Avoid cigarette burns and pet claws puncturing the mattress cover.

## 4.2 Preparing the Mattress for Use

**CAUTION!****Risk of Injury or Damage to Property**

- Before use, check the product for any defect.
- Do not use a defective product.

1. Remove all packaging before use.
2. Place the mattress directly on the mattress support of the bed or place the mattress overlay on the existing mattress.

Compress rolled mattresses must be left to recover before use. Follow the instructions on the packaging inlay.

Mattresses that are delivered flat packed can be used immediately.

## 5 Maintenance

### 5.1 Inspection

Check mattresses (foam and cover) for strike-through (this may include fluid ingress, stains, rips or damage) after the release of each patient, after ending of the period of use or on a minimum monthly basis (depending on which occurs first) by a suitably qualified and competent person.

**Check mattress**

1. Unzip the cover completely.
2. Check for any staining on the white underside of the cover.
3. Check for any staining on the interior foam.
4. Replace any stained items and dispose of as per local authority procedure.

## 5.2 Cleaning and Disinfection

### General Safety Information

**CAUTION!****Risk of Contamination**

- Take precautions for yourself and use appropriate protective equipment.

**IMPORTANT!**

- Wrong fluids or methods can harm or damage the product.
- All cleaning agents and disinfectants used must be effective, compatible with one another and must protect the materials they are used to clean.
  - Never use corrosive fluids (alkalines, acid etc.) or abrasive cleaning agents. We recommend an ordinary household cleaning agent such as dishwashing liquid, if not specified otherwise in the cleaning instructions.
  - Never use a solvent (cellulose thinner, acetone etc.) that changes the structure of the plastic or dissolves the attached labels.
  - Always make sure that the product is completely dried before taking it into use again.

For cleaning and disinfection in clinical or long-term care environments, follow your in-house procedures.

### Cleaning Intervals

**IMPORTANT!**

- Regular cleaning and disinfection enhances smooth operation, increases the service life and prevents contamination.
- Clean and disinfect the product
- regularly while it is in use,
  - before and after any service procedure,
  - when it has been in contact with any body fluids,
  - before using it for a new user.

**IMPORTANT!**

- Keep a cleaning record as part of cleaning the system.

### Cleaning Instructions

#### Cleaning Covers

(Removal of contaminants such as dust and organic matter)

1. Remove cover for laundering.
2. Launder the cover with maximum temperature as stated on product label, using a diluted detergent solution (Instructions on label).

**IMPORTANT!**

- Washing at higher temperatures will cause shrinkage.

#### Drying Covers

1. Hang cover from a line or bar and drip dry in a clean indoor environment.  
or  
Tumble dry on a low temperature.

**IMPORTANT!**

- Tumble dry setting must not exceed 40 °C.
- Do not tumble dry for longer than 10 minutes.
- Dry thoroughly before use.

## Disinfection Instructions

 Information on recommended disinfectants and methods can be found on <https://vah-online.de/en/for-users>.

Also look up your country's recommended disinfectants and methods.

### Disinfecting covers

(Reducing the number of microorganisms)

1. Wipe down the cover with a suitable detergent.
2. Rinse the cover with clean water using a single-use nonabrasive cloth.
3. Dry the cover thoroughly.

**!** **Heavy soilage**  
 Where the mattress is badly soiled, we recommend cleaning with a dilute cleaning solution at 95 °C in the washing machine.  
 – Clean up all spillages of bodily fluids i.e. blood, urine, faeces, sputum, wound exudate and all other bodily secretions as soon as possible using a suitable detergent.  
 – Large spillages of blood should be absorbed and removed with paper towels first.

### Autoclaving covers

Autoclave the cover at 110° C.

**!** **IMPORTANT!**  
 – Make sure the cover is outstretched (not folded).  
 – Do not place covers on top of each other.

### 5.3 Replacing Cover

1. Unzip cover and remove it carefully from foam insert.
2. Place new cover onto foam insert.
3. Close zipper.

**!** **IMPORTANT!**  
 – Ensure that the corners of the foam insert are positioned correctly into the corners of the cover.  
 – Ensure that the castellated foam is facing upwards when inserted into its cover.

## 7.1 General Data

The mattresses are available in various sizes, depending on the market. Contact your Invacare distributor for detailed information.

Local requirements / certification available upon request from Invacare distributor.

Product	Nominal density range [kg/m <sup>3</sup> ]	Nominal hardness range [N]	Maximum user weight [kg]	Weight of product [kg]
Essential Basic	31.4 – 34.7	120 – 150	140	9
Essential Care	31.4 – 34.7	120 – 150	110 / 140 <sup>1)</sup>	6 / 8 <sup>1)</sup>
Essential Wave	31.4 – 34.7	120 – 150	150	8
Essential Visco top base	31.4 – 34.7 48 – 52	120 – 150 46 – 64	110 / 140 <sup>1)</sup>	13 / 15 <sup>1)</sup>
Essential Plus	37 – 42	135 – 185	140	14
Essential Touch	37 – 42	135 – 185	150	14

<sup>1)</sup> Based on 120 mm / 150 mm height respectively.

### WARNING!

- In event of contamination, contact your hygiene specialist.
- Remove contaminated foams from use.

## 6 After Use

### 6.1 Storage

**!** **IMPORTANT!**  
 – Store mattresses in a dry environment.  
 – Store mattresses within a protective cover.  
 – Store items on clean, dry (off-floor) surface with no sharp edges to avoid any possible damage.  
 – Never store other items on top of a mattress.  
 – Do not store mattresses next to radiators or other heating devices.  
 – Protect mattresses from direct sunlight.

For environmental conditions for storage, see 7 *Technical Data*, page 5 .

### 6.2 Reconditioning

This product is suitable for reuse. To recondition the product for a new user, carry out the following actions:

- Inspection
- Cleaning and disinfection

For detailed information, see 5 *Maintenance*, page 4 .

Make sure that the user manual is handed over with the product.

If any damage or malfunction is detected, do not reuse the product.

### 6.3 Disposal

Be environmentally responsible and recycle this product through your recycling facility at its end of life.

Disassemble the product and its components, so the different materials can be separated and recycled individually.

The disposal and recycling of used products and packaging must comply with the laws and regulations for waste handling in each country. Contact your local waste management company for information.

## 7 Technical Data

## 7.2 Materials

Foam	Polyurethane Combustion Modified High Resilience Foam
Cover	Polyurethane transfer coated on weft knitted fabric

All product components are latex-free.

## 7.3 Environmental Conditions

	Operation	Storage and Transport
Ambient temperature	10 °C – 35 °C	-40 °C – 70 °C
Relative humidity	30% – 75%, non-condensing	10% – 100%, non-condensing
Atmospheric pressure	70 – 106 kPa	50 – 106 kPa

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Invacare Austria GmbH
Invacare Limited
Limited