

Spare Parts Catalogue

Version: 2016/12/31b EN





Yes, you can.®

General Information

For any order of spare parts the following information is required:

- Product type*
- Product name & model*
- Serial Number*
- Part-Number*
- Part-Description*

Exploded view drawings and code numbers are not contractual and subject to modification, in relation to Invacare® continuous product improvement policy to meet end users needs.

The reproduction or use of this work, or of any substantial part of this work, in any form or by any electronic, mechanical or other means, including photocopying, scanning and recording, and in any information storage and retrieval system, including electronic transmission, is forbidden without the prior written permission of Invacare® International Sarl.



Yes, you can.®

TABLE OF CONTENTS

Information & User Manuals

User Manuals Storm4	15
User Manuals Ultra Low Maxx Seat	16
User Manuals REM 550	17
User Manuals REM 24 SD	18
User Manuals REM A & B	19
User Manuals G90A	20
User Manuals G91S	21
User Manuals REM 34 & REM 41 Tray/Centre Control	22
User Manuals Chin Control (2015-10-01 =>)	23
User Manuals IDC (Intuitive Dual Control)	24
User Manuals Compact Dual Control	25
User Manual Proximity Head Remote DSL-105	26
User Manuals ACS Handprogrammer	27
Change list Recaro 24V	28

Chassis & Shrouds

Storm4	29
Chassis Fittings	30
Chassis Fittings (with Ultra Low Maxx Seat)	31
Front swingarm	32
Front Swing Arm Storm4 X-Plore	33
Motor Swingarm => 04.01.11	34
Motor Swingarm SSD	35
Motor Swingarm SSD (with Ultra Low Max Seat)	36
Motor Swingarm SSD, Anti-Tippers flip up	37
Rear suspension => 03.08.12	38
Rear suspension 04.08.12 =>	39
Upper suspension, Storm4 X-Plore	40
Transport fixation => 03.08.12	41
Transport fixation 04.08.12 =>	42
Shrouds	43
Shrouds (with Ultra Low Maxx Seat)	45
Shroud spring bracket/rear light	46
Storm4 TTPlus	47
Chassis Fittings	30
Front swingarm	32
Front Swing Arm Storm4 X-Plore	33
Motor Swingarm (GGB)	48
Suspension	49
Upper suspension, Storm4 X-Plore	40
Transport Fixation	50
Shrouds	51
Shrouds Spring Bracket	52

Seat

Seat	53
Storm4	54
Standard Seat	55
Seat column / -raiser Storm4	56



Yes, you can.[®]

TABLE OF CONTENTS

Seat column / -raiser Storm4 X-Plore	57
Seat Height Adapter Storm4	58
Seat Height Adapter Storm4 X-Plore	59
Seat Tilt	60
Seat Frame	61
Seat Plate	62
Seat Cushion	63
EasyAdapt Seat	64
Seat column / -raiser Storm4	56
Seat column / -raiser Storm4 X-Plore	57
Seat Height Adapter Storm4	58
Seat Height Adapter Storm4 X-Plore	59
Seat Tilt	60
Seat Frame	61
Seat Plate	62
Adapter Kit for Universal Seat	65
Seat Cushion	66
Modulite Seat	68
Seat column / -raiser Storm4	56
Seat column / -raiser Storm4 X-Plore	57
Seat Height Adapter Storm4	58
Seat Height Adapter Storm4 X-Plore	59
Seat Tilt	69
One Piece Seat Plate	70
Telescopic Seat Frame and Plate	71
Telescopic Seat Frame	72
Armrest bracket	73
Legrest Support	74
Seat Plate	75
Telescopic Seat Frame and Sling Seat	76
Telescopic Seat Frame	77
Armrest bracket	73
Legrest Support	78
Sling Seat	79
Telescopic Seat Frame and Universal Adapter	80
Telescopic Seat Frame	72
Armrest bracket	73
Legrest Support	74
Adapter Kit for Universal Seat	81
Seat Cushions "Standard" & "Matrx Flo-Tech"	82
Seat Cushions "Matrx Libra"	85
Seat Cushions "Matrx Vi"	86
Ultra Low Maxx Seat	87
Seat Column	88
Seat Tilt	89
Seat Lifter with Tilt	90
Posterior-Tilt	91
Seat Plates & Side Rails	92
Seat Cushions "Matrx Vi"	93
Seat Cushions "Matrx Flo-Tech"	94
Seat Cushions "Matrx Libra"	95
Recaro Seat	96



Yes, you can.®

TABLE OF CONTENTS

Seat column / -raiser Storm4	56
Seat column / -raiser Storm4 X-Plore	57
Seat Height Adapter Storm4	58
Seat Height Adapter Storm4 X-Plore	59
Seat Tilt	60
Seat Frame	97
Seat Plate	98
Recaro Seat	99
Change list Recaro 24V	28
Optimist Seat	100
Seat column / -raiser Storm4	56
Seat column / -raiser Storm4 X-Plore	57
Seat Height Adapter Storm4	58
Seat Height Adapter Storm4 X-Plore	59
Seat Tilt	60
Seat Plate	101
Seat Frame	102
Seat Cushions Optimist Seat	103
Storm4 TTPlus	104
EasyAdapt Seat	105
Seat column / Seatrisher	106
Seat Height Adapter	107
Seat Tilt	108
Seat Frame	61
Seat Plate	62
Adapter Kit for Universal Seat	65
Seat Cushion	66
Recaro Seat	109
Seat column / Seatrisher	106
Seat Height Adapter	107
Seat Tilt	108
Seat Frame	97
Seat Plate	98
Recaro Seat	99
Change list Recaro 24V	28
Optimist Seat	110
Seat column / Seatrisher	106
Seat Height Adapter	107
Seat Tilt	108
Seat Plate	101
Seat Frame	102
Seat Cushions Optimist Seat	103
Modulite Seat	111
Seat column / Seatrisher	106
Seat Height Adapter	107
Seat Tilt	112
One Piece Seat Plate	70
Telescopic Seat Frame and Plate	71
Telescopic Seat Frame	72
Armrest bracket	73
Legrest Support	74
Seat Plate	75



Yes, you can.[®]

TABLE OF CONTENTS

Telescopic Seat Frame and Sling Seat	76
Telescopic Seat Frame	77
Armrest bracket	73
Legrest Support	78
Sling Seat	79
Telescopic Seat Frame and Universal Adapter	80
Telescopic Seat Frame	72
Armrest bracket	73
Legrest Support	74
Adapter Kit for Universal Seat	81
Seat Cushions "Standard" & "Matrx Flo-Tech"	82
Seat Cushions "Matrx Libra"	85
Seat Cushions "Matrx Vi"	86

Backrest

Backrest	113
Storm4	114
Backrest for Standard Seat	115
Standard Backrest	116
Backrest for EasyAdapt Seat	117
Backrestadjusting	118
Backrestcover for EasyAdapt	119
Backplate EasyAdapt	120
Backplate EasyAdapt for DAHL Docking Station	121
Backrest Cushions	122
Backrest Cushions Laguna & Onyx	123
Matrx backrest	124
Matrx Posture Back	125
Matrx Elite (Full) Back	126
Mounting Hardware for Matrx backrests	127
Push Handle EasyAdapt	128
Tension adjustable Sling Back	129
"Universal"-Backrest frame	130
Backrest for Modulite Seat	131
Support for Backrest Frame	132
Backrest Frame	133
Sling Back, tension adjustable	134
Backrest support	135
Backposts	136
Connection tube	137
Back Recliner / Stabilization bracket	138
Backrest Cushions	139
Flex 3 / Laguna	140
Matrx Posture Back	125
Matrx Elite T/R Back	142
Matrx Elite Deep Back	143
Matrx Elite (Full) Back	126
Mounting Hardware for Matrx backrests	127
Flex3 back, tension adjustable	144
Backrest for Ultra Low Maxx Seat	145
Backrest Recliner	146
High Back	147



Yes, you can.®

TABLE OF CONTENTS

Backrest Hardware	148
Backrest Cushions	149
Matrx Back	151
Backrest Hardware	152
Posture Back	153
Elite T/R Back	154
Elite Deep Back	155
Privacy Flaps	156
Backrest for Optimist Seat	157
Optimist Seats	158
Push Handle Optimist Seat	159
Storm4 TTPlus	160
Backrest for Standard Seat	115
Standard Backrest	116
Backrest for EasyAdapt Seat	117
Backrestadjusting	118
Backrestcover for EasyAdapt	119
Backplate EasyAdapt	120
Backplate EasyAdapt for DAHL Docking Station	121
Backrest Cushions	122
Backrest Cushions Laguna & Onyx	123
Matrx backrest	124
Matrx Posture Back	125
Matrx Elite (Full) Back	126
Mounting Hardware for Matrx backrests	127
Push Handle EasyAdapt	128
Tension adjustable Sling Back	129
"Universal"-Backrest frame	130
Backrest for Modulite Seat	161
Support for Backrest Frame	132
Backrest Frame	133
Sling Back, tension adjustable	134
Backrest support	135
Backposts	136
Connection tube	137
Backrest Recliner / Stabilisation bracket	162
Backrest Cushions	139
Flex 3 / Laguna	140
Matrx Posture Back	125
Matrx Elite T/R Back	142
Matrx Elite Deep Back	143
Matrx Elite (Full) Back	126
Mounting Hardware for Matrx backrests	127
Flex3 back, tension adjustable	144
Backrest for Optimist Seat	157
Optimist Seats	158
Push Handle Optimist Seat	159

Legrests

Storm4	163
Legrests for Ultra Low Maxx Seat	164
Central Legrests	165



Yes, you can.®

TABLE OF CONTENTS

Supports, Central Legrests	166
Central Legrest pow. (LNX)	167
Central Legrest man.	168
Vari Legrests	169
Supports, Vari	170
Hanger Vari A (04/2009 =>)	171
Hanger Vari F	172
Hangers ADM LH/RH manual	173
Hanger Vari ADE/ADP electric (07/2005 =>)	174
Footrests Vari, Flip-Up (01.08.10 =>)	175
Footrests Vari, Flip-Up & angle adjustable (01.08.10 =>)	176
Footrests Vari, angle & depth adjustable	177
Pivot Plus Legrests	178
Cross Brace, Pivot Plus	179
Legrests	180
Legrest Support, Vari series 07/2005 ==>	181
Hanger Vari A (04/2009 =>)	171
Hanger Vari F	172
Hangers ADM LH/RH manual	173
Hanger Vari ADE/ADP electric (07/2005 =>)	174
Central Legrest, manuell	182
Central Legrest, electric	183
Footrests Vari, Flip-Up (01.08.10 =>)	175
Footrests Vari, Flip-Up & angle adjustable (01.08.10 =>)	176
Support Tube with Footplate for central Legrest	184
Lower Footrests	185
Storm4 TTPlus	186
Legrest Supports Vari (07/2005 =>)	187
Hanger Vari A (04/2009 =>)	171
Hanger Vari F	172
Hangers ADM LH/RH manual	173
Hanger Vari ADE/ADP electric (07/2005 =>)	174
Central Legrest, manuell	182
Central Legrest, electric	188
Footrests Vari, Flip-Up (01.08.10 =>)	175
Footrests Vari, Flip-Up & angle adjustable (01.08.10 =>)	176
Support Tube with Footplate for central Legrest	184
Lower Footrests	185

Armrests

Armrests	189
Armrest for Standard Seat	190
Fixed Armrest, removable	191
Armrests for EasyAdapt Seat	192
Fixed Armrest, removable	191
Following armrests	193
Skirtguard for following armrest	194
Armrests for Modulite Seat	195
T-Armrest	196
Padded Armrest	197
Inrigged Armrest	198
Flip-Up Armrest	199



Yes, you can.®

TABLE OF CONTENTS

Following Armrest	200
Hip Support	201
Armrests for Ultra Low Maxx Seat	202
Recline Flip-Up Armrest	203
Cantilever Flip-Up Armrest	204
Armpads, Desk & Full	205
Armpad, Ergonomic	206
Elbow Block	207
MACES Multi Axis Armpad Holder	208
Hip Support	201
Armrest for Recaro Seat	209
Fixed Armrest, removable	191
Armrests for Optimist Seat	210
Armrests Arne Jensen for Optimist Seat	211

Castors

Front Fork	212
Front Wheels	213

Drive Wheels

Storm4	214
Drive Wheel Trelleborg (8x3.00)	215
Drive Wheel 14" (3.00 - 8)	216
Storm4 TTPlus	217
Drive Wheels GGB	218

Propulsion

Storm4	219
Propulsion => 04.01.11	220
Overview Propulsion SSD	221
Storm4 TTPlus	224
GGB2 Motors	225

Electronic

Electronic	226
Storm4	227
ACS Electronic	228
ACS Electronic Modules	229
Power modules ACS 2	230
Holder Powermodule	231
Power module holder / Gyro	232
ACTs	233
Bracket ACT - Ultra Low Maxx Seat	234
G-Trac Sensor/Gyro	235
Holder Gyro	236
Light board	237
ACS Remotes & brackets	238
REM 24 SD with brackets	239
Remote REM 24SD	240
Remote Bracket Standard	241



Yes, you can.®

TABLE OF CONTENTS

Remote Bracket Standard - ULM and Soft Armpad on Modulite	242
Remote Bracket Swing away & height adjust.	243
Remote Bracket Swing away & height adjust. - ULM and Soft Armpad on Modulite	244
Remote Bracket Attendant	245
Remote Bracket Attendant Ultra Low Maxx Seat	246
Remote Bracket Attendant for Recaro-Seat	247
REM A & B with brackets	248
Remote Bracket Standard	241
Remote Bracket Swing away & height adjust.	249
Remote Bracket Attendant for Standard-/EasyAdapt-Seat	250
Remote Bracket Attendant for Recaro-Seat	247
REM 550 with brackets	251
Remote Bracket Standard	241
Remote Bracket Standard - ULM and Soft Armpad on Modulite	242
Remote Bracket Swing away & height adjust.	243
Remote Bracket Swing away & height adjust. - ULM and Soft Armpad on Modulite	244
Remote Bracket Attendant	245
Remote Bracket Attendant Ultra Low Maxx Seat	246
Remote Bracket Swing away Quad Link Ultra Low Maxx Seat	252
Remote Bracket Attendant for Recaro-Seat	247
REM 34 with brackets	253
Remote Bracket Standard	254
Remote Bracket Swing away & height adjust.	255
G90A with brackets	256
Remote Bracket Standard	257
Remote Bracket Standard - ULM and Soft Armpad on Modulite	258
Remote Bracket Swing away & height adjust.	259
Remote Bracket Swing away & height adjust. - ULM and soft Armpad on Modulite	260
Remote Bracket Attendant	261
Remote Bracket Attendant Ultra Low Maxx Seat	262
Remote Bracket Attendant for Recaro-Seat	263
G91S with brackets	264
Remote Bracket Standard	265
Centre control REM 41	266
Chin Control, small (=> 16/12/31)	267
Small Chincontrol	268
Framework for chincontrol	269
Chin Control - Modulite Seat (15/10/01 =>)	270
User Manuals Chin Control (2015-10-01 =>)	23
Chin Control DX-RJM	271
Chin Control powered - Details	272
Chin Control manual - Details	273
Mounting Hardware - Chin Control	274
Dual Control with brackets	275
Remote Bracket Dual Control for Standard-/EasyAdapt-Seat	276
Remote Bracket Dual Control for Modulite Seat	277
Remote Bracket Dual Control for Recaro Seat	278
Compact Dual Control with brackets	279
Compact Dual Control	280
Bracket, Compact Dual Control - Ultra Low Maxx Seat	281
Bracket, Compact Dual Control - short	282
Bracket, Compact Dual Control - long	283



Yes, you can.®

TABLE OF CONTENTS

Intuitive Dual Control (IDC) with brackets	284
IDC with bracket	285
User Manuals IDC (Intuitive Dual Control)	24
IDC with bracket (Ultra Low Maxx Seat)	286
User Manuals IDC (Intuitive Dual Control)	24
Height adjustable bracket IDC	287
ACS Cables & Accessories	288
Bus-Cables ACS	289
T-Adapter	290
T-Adapter (Ultra Low Maxx Seat)	291
Backrest Sensor - Ultra Low Maxx Seat	292
Tilt Sensor - Ultra Low Maxx Seat	293
Programming Kits	294
Magnetkey for Remotes	295
ACS Components	296
iPortal Modules and Accessories	297
Remote, Joystick, Control	298
Joystick Knob	299
Buddy Button	300
Piko Button	301
Interface, Adapter, Other	302
LiNX Electronic	303
Modules	304
Power Modules	305
Bracket, Power Module	306
Actuator Modules (ACTs)	307
G-Trac Sensor / Gyro	308
Remotes & Brackets	309
User Manuals LiNX	310
Remotes & Brackets	311
Remote REM110	312
Remote REM211	313
Remote REM216	314
Remote Bracket, Standard	315
Remote Bracket, Swing away	316
Remote Bracket, Attendant	317
Dual Control & Bracket	318
Dual Control	319
Remote Bracket, Attendant	317
Cables & Accessories	320
Bus Cables	321
Actuator Cable	322
Lifter/Tilt-Module Cables	323
Light Cable	324
Light Board	325
4-way Expansion Block	326
Slow-Stop-Sensor - Modulite Seat	327
Slow-Stop-Sensor - Recaro Seat	328
Components	329
Accessory cables	330



Yes, you can.®

TABLE OF CONTENTS

10-way switch & brackets

10-way switch & brackets	331
10-way switch	332
Bracket, front	333
Bracket, opposite side	334
Bracket, rear	335
Buddy Button	300
Piko Button	301
Storm4 TTPlus	336
ACS Modules	337
Powermodule GGB & Bracket	338
CLAM & Bracket	339
Relais & bracket	340
Lightboard GGB	341
ACS Remotes & Brackets	342
REM 24 SD with brackets	343
Remote REM 24SD	240
Remote Bracket Standard	241
Remote Bracket Swing away & height adjust.	243
Remote Bracket Attendant for Standard-/EasyAdapt-Seat	250
Remote Bracket Attendant for Recaro-Seat	247
REM A & B with brackets	248
Remote Bracket Standard	241
Remote Bracket Swing away & height adjust.	249
Remote Bracket Attendant for Standard-/EasyAdapt-Seat	250
Remote Bracket Attendant for Recaro-Seat	247
REM 550 with brackets	344
Remote Bracket Standard	241
Remote Bracket Swing away & height adjust.	243
Remote Bracket Attendant for Standard-/EasyAdapt-Seat	250
Remote Bracket Attendant for Recaro-Seat	247
REM 34 with brackets	253
Remote Bracket Standard	254
Remote Bracket Swing away & height adjust.	255
G90A with brackets	345
Remote Bracket Standard	257
Remote Bracket Swing away & height adjust.	259
Remote Bracket Attendant for Standard-/EasyAdapt-Seat	346
Remote Bracket Attendant for Recaro-Seat	263
G91S with brackets	264
Remote Bracket Standard	265
Centre control REM 41	266
Chin Control, small (=> 16/12/31)	267
Small Chincontrol	268
Framework for chincontrol	269
Chin Control - Modulte Seat (15/10/01 =>)	270
User Manuals Chin Control (2015-10-01 =>)	23
Chin Control DX-RJM	271
Chin Control powered - Details	272
Chin Control manual - Details	273
Mounting Hardware - Chin Control	274



Yes, you can.[®]

TABLE OF CONTENTS

Dual Control with brackets	275
Remote Bracket Dual Control for Standard-/EasyAdapt-Seat	276
Remote Bracket Dual Control for Modulite Seat	277
Remote Bracket Dual Control for Recaro Seat	278
Compact Dual Control with brackets	347
Compact Dual Control	280
Bracket, Compact Dual Control - short	282
Bracket, Compact Dual Control - long	283
Intuitive Dual Control (IDC) with bracket	348
IDC with bracket	285
User Manuals IDC (Intuitive Dual Control)	24
Height adjustable bracket IDC	287
ACS Cables & Accessories	349
Main cable harness	350
Plug Sheets el. legrest GGB	351
T-Adapter	352
T-Adapter Modulite Seat	353
Programming Kits	294
Magnetkey for Remotes	295
ACS Components	296
iPortal Modules and Accessories	297
Remote, Joystick, Control	298
Joystick Knob	299
Buddy Button	300
Piko Button	301
Interface, Adapter, Other	302

Batteries & Chargers

Storm4	354
Batteries & Accessories	355
Chargers	356
Storm4 TTPlus	357
Batteries & Accessories	358
Chargers	356

Safety Options & Accessories

Frontlight (German homologation)	359
Frontlight for Modulite (German homologation)	360
Rearlight (German homologation)	361
Rear Light Protector Kit	362
LED-Frontlight	363
LED-Frontlight for Modulite	364
LED-Rearlight	365
Attachment kit optimist seat	366
Reflective Sign	367
Reflective Sign (Ultra Low Maxx Seat)	368
Kerb Rider	369
Lap Belts / H-Belt	370
Lap Belts (Modulite Seat)	371
H-Belts (Modulite Seat)	372
Lap Belt (Ultra Low Maxx)	373



Yes, you can.®

TABLE OF CONTENTS

Lap Belt (Max Seat)	374
Rearview Mirror	375
Anti-Tippers	376
Storm4	377
Anti-Tipper Wheels => 04.01.11	378
Anti-Tipper for SSD Motors	379
Anti-Tippers for SSD Motors, flip up	380
Storm4 TTP	381
Anti-Tippers for GGB2 Motors	382

Seat Accessories

Headrests	383
Headrest with cheek support	384
Headrest & Bracket	385
Headrestholder EasyAdapt Seat	386
Headrest, sidewise adjustable	387
Headrests - Ultra Low Maxx Seat	388
Lateral Support	389
Lateral Supports - Modulite Seat	390
Lateral supports, Flex3 backrest	391
Lateral supports, Flex3 backrest - multi adjustable	392
Lateral supports, Flex3 backrest (tension adjustable) - multi adjustable	393
Lateral Support - Ultra Low Maxx Seat	394
Lateral Support - EasyAdapt Seat	395
Lateral Support (Dartex)	396
Lateral Support, multi adjustable with flexible pad	397
Bag	398

Accessories

Luggage Rack	399
Mounting Kit Luggage Rack	400
Caneholder	401
Cane holder (Easy Adapt Seat & Std-Seat)	402
Cane holder (Recaro Seat)	403
Cane holder (Modulite: backrest frame)	404
Cane holder (Modulite: back posts man recliner)	405
Cane holder (Modulite: back posts pow recliner)	406
Fender for Castors	407
Fenders for Drive Wheels => 04.01.11	408
Fenders for Drive Wheels (SSD-Motors)	409
Fenders (GGB Motors)	410
Trays	411
Trays	412
Trays (Ultra Low Maxx Seat)	414
km-Counter & Bracket	415
Mounting-kit km-Counter	416
Hour Counter	417
Bag	418
Toolkit	419
T-Nuts for Accessories Modulite & ULM Seat	420

User Manuals Storm4

languages.iso

BG	Български	FR	Français	NL	Nederlands	SL	Slovenščina
CS	Čeština	HE	עברית	NO	Norsk	SV	Svenska
DA	Dansk	HR	Hrvatski	PL	Polski	UK	Українська
DE	Deutsch	HU	Magyar	PT	Português		
EN	English	IT	Italiano	RU	Русский		
ES	Español	JA	日本語	RO	Română		
ET	Eesti	LV	Latviešu	SA	العربية		
FI	Suomi	LT	Lietuvių	SK	Slovenčina		

Pos.	Part number	Description	Valid from → until	Qty
	CS 1532570	User Manual Storm4 Czech		1
	DA 1529704	User Manual Storm4 Danish		1
	DE 1529687	User Manual Storm4 German		1
	EN 1529686	User Manual Storm4 English		1
	ES 1529690	User Manual Storm4 Spanish		1
	ET SP1543013	User Manual Storm4 Estonian		1
	FI 1529702	User Manual Storm4 Finsk		1
	FR 1529688	User Manual Storm4 French		1
	HU SP1574045	User Manual Storm4 Hungarian		1
	IT 1529689	User Manual Storm4 Italian		1
	LT 1532635	User Manual Storm4 Lithuanian		1
	NL 1529700	User Manual Storm4 Dutch		1
	NO 1529703	User Manual Storm4 Norsk		1
	PL 1532571	User Manual Storm4 Polish		1
	RU SP1549066	User Manual Storm4 Russian		1
	SL 1532634	User Manual Storm4 Slovenian		1
	SV 1529701	User Manual Storm4 Swedish		1

User Manuals Ultra Low Maxx Seat

languages.iso

BG	Български	FR	Français	NL	Nederlands	SL	Slovenščina
CS	Čeština	HE	עברית	NO	Norsk	SV	Svenska
DA	Dansk	HR	Hrvatski	PL	Polski	UK	Українська
DE	Deutsch	HU	Magyar	PT	Português		
EN	English	IT	Italiano	RU	Русский		
ES	Español	JA	日本語	RO	Română		
ET	Eesti	LV	Latviešu	SA	العربية		
FI	Suomi	LT	Lietuvių	SK	Slovenčina		

Pos.	Part number	Description	Valid from → until	Qty
DA	SP1585726	User Manual Ultra Low Maxx Seat - Danish		1
DE	SP1585727	User Manual Ultra Low Maxx Seat - German		1
EN	SP1585725	User Manual Ultra Low Maxx Seat - English		1
ES	SP1601887	User Manual Ultra Low Maxx Seat - Spanish		1
FI	SP1585728	User Manual Ultra Low Maxx Seat - Finish		1
FR	SP1585729	User Manual Ultra Low Maxx Seat - French		1
IT	SP1588266	User Manual Ultra Low Maxx Seat - Italian		1
NL	SP1585730	User Manual Ultra Low Maxx Seat - Dutch		1
NO	SP1585731	User Manual Ultra Low Maxx Seat - Norwegian		1
PT	SP1585732	User Manual Ultra Low Maxx Seat - Portuguese		1
SV	SP1585733	User Manual Ultra Low Maxx Seat - Swedish		1

User Manuals REM 550

languages.iso

BG	Български	FR	Français	NL	Nederlands	SL	Slovenščina
CS	Čeština	HE	עברית	NO	Norsk	SV	Svenska
DA	Dansk	HR	Hrvatski	PL	Polski	UK	Українська
DE	Deutsch	HU	Magyar	PT	Português		
EN	English	IT	Italiano	RU	Русский		
ES	Español	JA	日本語	RO	Română		
ET	Eesti	LV	Latviešu	SA	العربية		
FI	Suomi	LT	Lietuvių	SK	Slovenčina		

Pos.	Part number	Description	Valid from → until	Qty
IT	1528094	User Manual REM550 Italiano		1
NO	1528096	User Manual REM550 Norsk		1
ES	1528091	User Manual REM550 Spanish		1
FI	1528092	User Manual REM550 Finsk		1
EN	1528090	User Manual REM550 English		1
DA	1528088	User Manual REM550 Danish		1
NL	1528095	User Manual REM550 Dutch		1
DE	1528089	User Manual REM550 German		1
PT	1528097	User Manual REM550 Portuguese		1
SV	1528098	User Manual REM550 Swedish		1
FR	1528093	User Manual REM550 French		1
CS	SP1533280	User Manual REM550 Czech		1
ET	SP1533281	User Manual REM550 Estonian		1
HR	SP1533279	User Manual REM550 Croatian		1
HU	SP1533282	User Manual REM550 Hungarian		1
LT	SP1533283	User Manual REM550 Lithuanian		1
PL	SP1533284	User Manual REM550 Polish		1
SL	SP1533285	User Manual REM550 Slovenian		1

User Manuals REM 24 SD

languages.iso

BG	Български	FR	Français	NL	Nederlands	SL	Slovenščina
CS	Čeština	HE	עברית	NO	Norsk	SV	Svenska
DA	Dansk	HR	Hrvatski	PL	Polski	UK	Українська
DE	Deutsch	HU	Magyar	PT	Português		
EN	English	IT	Italiano	RU	Русский		
ES	Español	JA	日本語	RO	Română		
ET	Eesti	LV	Latviešu	SA	العربية		
FI	Suomi	LT	Lietuvių	SK	Slovenčina		

Pos.	Part number	Description	Valid from → until	Qty
	CS SP1533252	User Manual REM24SD Czech		1
	DA 1436146	User Manual REM24SD Danish		1
	DE 1436142	User Manual REM24SD German		1
	EN 1436144	User Manual REM24SD English		1
	ES 1436149	User Manual REM24SD Spanish		1
	ET SP1533253	User Manual REM24SD Estonian		1
	FI 1436386	User Manual REM24SD Finnish		1
	FR 1436143	User Manual REM24SD French		1
	HR SP1533251	User Manual REM24SD Croatian		1
	HU SP1533254	User Manual REM24SD Hungarian		1
	IT 1436385	User Manual REM24SD Italian		1
	LV SP1543152	User Manual REM24SD Latvian		1
	LT SP1533255	User Manual REM24SD Lithuanian		1
	NL 1436148	User Manual REM24SD Dutch		1
	NO 1436147	User Manual REM24SD Norwegian		1
	PL SP1533256	User Manual REM24SD Polish		1
	PT 1436150	User Manual REM24SD Portuguese		1
	SL SP1533257	User Manual REM24SD Slovenian		1
	SV 1436145	User Manual REM24SD Swedish		1

User Manuals REM A & B

languages.iso

BG	Български	FR	Français	NL	Nederlands	SL	Slovenščina
CS	Čeština	HE	עברית	NO	Norsk	SV	Svenska
DA	Dansk	HR	Hrvatski	PL	Polski	UK	Українська
DE	Deutsch	HU	Magyar	PT	Português		
EN	English	IT	Italiano	RU	Русский		
ES	Español	JA	日本語	RO	Română		
ET	Eesti	LV	Latviešu	SA	العربية		
FI	Suomi	LT	Lietuvių	SK	Slovenčina		

Pos.	Part number	Description	Valid from → until	Qty
	IT 1529230	User Manual REM A & B - Italian		1
	NO 1529232	User Manual REM A & B - Norwegian		1
	ES 1529174	User Manual REM A & B - Spanish		1
	FI 1529228	User Manual REM A & B - Finnish		1
	EN 1529173	User Manual REM A & B - English		1
	DA 1529171	User Manual REM A & B - Danish		1
	NL 1529231	User Manual REM A & B - Dutch		1
	DE 1529172	User Manual REM A & B - German		1
	PT 1529233	User Manual REM A & B - Portuguese		1
	SV 1529234	User Manual REM A & B Swedish		1
	FR 1529229	User Manual REM A & B - French		1
	CS SP1533259	User Manual REM A & B - Czech		1
	ET SP1533265	User Manual REM A & B - Estonian		1
	HR SP1533258	User Manual REM A & B - Croatian		1
	HU SP1533266	User Manual REM A & B - Hungarian		1
	LT SP1533267	User Manual REM A & B - Lithuanian		1
	PL SP1533268	User Manual REM A & B - Polish		1
	RO SP1533270	User Manual REM A & B - Romanian		1
	SL SP1533269	User Manual REM A & B - Slovenian		1

User Manuals G90A

languages.iso

BG	Български	FR	Français	NL	Nederlands	SL	Slovenščina
CS	Čeština	HE	עברית	NO	Norsk	SV	Svenska
DA	Dansk	HR	Hrvatski	PL	Polski	UK	Українська
DE	Deutsch	HU	Magyar	PT	Português		
EN	English	IT	Italiano	RU	Русский		
ES	Español	JA	日本語	RO	Română		
ET	Eesti	LV	Latviešu	SA	العربية		
FI	Suomi	LT	Lietuvių	SK	Slovenčina		

Pos.	Part number	Description	Valid from → until	Qty
EN	SP1524441	User Manual G90A - English		1
DE	SP1536742	User Manual G90A - German		1
ES	SP1638870	User Manual G90A - Spanish		1
FR	SP1638871	User Manual G90A - French		1
NO	SP1638872	User Manual G90A - Norwegian		1
PT	SP1638873	User Manual G90A - Portuguese		1
SV	SP1638874	User Manual G90A - Swedish		1
DA	SP1638875	User Manual G90A - Danish		1
NL	SP1638876	User Manual G90A - Dutch		1

User Manuals G91S

languages.iso

BG	Български	FR	Français	NL	Nederlands	SL	Slovenščina
CS	Čeština	HE	עברית	NO	Norsk	SV	Svenska
DA	Dansk	HR	Hrvatski	PL	Polski	UK	Українська
DE	Deutsch	HU	Magyar	PT	Português		
EN	English	IT	Italiano	RU	Русский		
ES	Español	JA	日本語	RO	Română		
ET	Eesti	LV	Latviešu	SA	العربية		
FI	Suomi	LT	Lietuvių	SK	Slovenčina		

Pos.	Part number	Description	Valid from → until	Qty
------	-------------	-------------	--------------------	-----

User Manuals REM 34 & REM 41 Tray/Centre Control

languages.iso

BG	Български	FR	Français	NL	Nederlands	SL	Slovenščina
CS	Čeština	HE	עברית	NO	Norsk	SV	Svenska
DA	Dansk	HR	Hrvatski	PL	Polski	UK	Українська
DE	Deutsch	HU	Magyar	PT	Português		
EN	English	IT	Italiano	RU	Русский		
ES	Español	JA	日本語	RO	Română		
ET	Eesti	LV	Latviešu	SA	العربية		
FI	Suomi	LT	Lietuvių	SK	Slovenčina		

Pos.	Part number	Description	Valid from → until	Qty
CS	SP1538031	User Manual REM41 Tray Control Czech		1
DA	SP1538022	User Manual REM41 Tray Control Danish		1
DE	SP1541402	User Manual REM41 Tray Control German		1
EN	SP1538023	User Manual REM41 Tray Control English		1
ES	SP1538024	User Manual REM41 Tray Control Spanish		1
FR	SP1538025	User Manual REM41 Tray Control French		1
IT	SP1538026	User Manual REM41 Tray Control Italian		1
NL	SP1538027	User Manual REM41 Tray Control Dutch		1
NO	SP1538028	User Manual REM41 Tray Control Norwegian		1
PT	SP1538029	User Manual REM41 Tray Control Portuguese		1
SL	SP1538032	User Manual REM41 Tray Control Slovenian		1
SV	SP1538030	User Manual REM41 Tray Control Swedish		1

User Manuals Chin Control (2015-10-01 =>)

languages.iso

BG	Български	FR	Français	NL	Nederlands	SL	Slovenščina
CS	Čeština	HE	עברית	NO	Norsk	SV	Svenska
DA	Dansk	HR	Hrvatski	PL	Polski	UK	Українська
DE	Deutsch	HU	Magyar	PT	Português		
EN	English	IT	Italiano	RU	Русский		
ES	Español	JA	日本語	RO	Română		
ET	Eesti	LV	Latviešu	SA	العربية		
FI	Suomi	LT	Lietuvių	SK	Slovenčina		

Pos.	Part number	Description	Valid from → until	Qty
	CS SP1589326	User Manual Chin Control - Czech		1
	DA SP1589394	User Manual Chin Control - Danish		1
	DE SP1589395	User Manual Chin Control - German		1
	EN SP1589325	User Manual Chin Control - English		1
	ES SP1589535	User Manual Chin Control - Spanish		1
	FI SP1589436	User Manual Chin Control - Finnish		1
	FR SP1589437	User Manual Chin Control - French		1
	HR SP1589438	User Manual Chin Control - Croatian		1
	IT SP1589439	User Manual Chin Control - Italian		1
	LT SP1589440	User Manual Chin Control - Lithuanian		1
	NL SP1589441	User Manual Chin Control - Dutch		1
	NO SP1589442	User Manual Chin Control - Norwegian		1
	PL SP1589443	User Manual Chin Control - Polish		1
	PT SP1589444	User Manual Chin Control - Portuguese		1
	SL SP1589445	User Manual Chin Control - Slovenian		1
	SV SP1589465	User Manual Chin Control - Swedish		1

User Manuals IDC (Intuitive Dual Control)

languages.iso

BG	Български	FR	Français	NL	Nederlands	SL	Slovenščina
CS	Čeština	HE	עברית	NO	Norsk	SV	Svenska
DA	Dansk	HR	Hrvatski	PL	Polski	UK	Українська
DE	Deutsch	HU	Magyar	PT	Português		
EN	English	IT	Italiano	RU	Русский		
ES	Español	JA	日本語	RO	Română		
ET	Eesti	LV	Latviešu	SA	العربية		
FI	Suomi	LT	Lietuvių	SK	Slovenčina		

Pos.	Part number	Description	Valid from → until	Qty
	DE 1521480	User Manual IDC - German		1
	EN 1521481	User Manual IDC - English		1
	FR 1521482	User Manual IDC - French		1
	NL 1521483	User Manual IDC - Dutch		1
	NO 1521484	User Manual IDC - Norwegian		1
	SV 1521485	User Manual IDC - Swedish		1
	DA 1521486	User Manual IDC - Danish		1
	IT 1522252	User Manual IDC - Italian		1
	PT 1522254	User Manual IDC - Portuguese		1
	ES 1522255	User Manual IDC - Spanish		1

User Manuals Compact Dual Control

languages.iso

BG	Български	FR	Français	NL	Nederlands	SL	Slovenščina
CS	Čeština	HE	עברית	NO	Norsk	SV	Svenska
DA	Dansk	HR	Hrvatski	PL	Polski	UK	Українська
DE	Deutsch	HU	Magyar	PT	Português		
EN	English	IT	Italiano	RU	Русский		
ES	Español	JA	日本語	RO	Română		
ET	Eesti	LV	Latviešu	SA	العربية		
FI	Suomi	LT	Lietuvių	SK	Slovenčina		

Pos.	Part number	Description	Valid from → until	Qty
BG	SP1589186	User Manual Compact Dual Control - Bulgarian		1
CS	SP1589313	User Manual Compact Dual Control - Czech		1
DA	SP1588488	User Manual Compact Dual Control - Danish		1
DE	SP1588489	User Manual Compact Dual Control - German		1
EN	SP1588462	User Manual Compact Dual Control - English		1
ES	SP1588496	User Manual Compact Dual Control - Spanish		1
ET	SP1589311	User Manual Compact Dual Control - Estonian		1
FI	SP1588490	User Manual Compact Dual Control - Finnish		1
FR	SP1588491	User Manual Compact Dual Control - French		1
HR	SP1589314	User Manual Compact Dual Control - Croatian		1
HU	SP1589315	User Manual Compact Dual Control - Hungarian		1
IT	SP1588492	User Manual Compact Dual Control - Italian		1
LV	SP1589317	User Manual Compact Dual Control - Latvian		1
LT	SP1589316	User Manual Compact Dual Control - Lithuanian		1
NL	SP1588493	User Manual Compact Dual Control - Dutch		1
NO	SP1588494	User Manual Compact Dual Control - Norwegian		1
PL	SP1589318	User Manual Compact Dual Control - Polish		1
PT	SP1588495	User Manual Compact Dual Control - Portuguese		1
RU	SP1589320	User Manual Compact Dual Control - Russian		1
RO	SP1589319	User Manual Compact Dual Control - Romanian		1
SK	SP1589321	User Manual Compact Dual Control - Slovak		1
SL	SP1589322	User Manual Compact Dual Control - Slovenian		1
SV	SP1588497	User Manual Compact Dual Control - Swedish		1
UK	SP1589323	User Manual Compact Dual Control - Ukrainian		1

User Manual Proximity Head Remote DSL-105

languages.iso

BG	Български	FR	Français	NL	Nederlands	SL	Slovenščina
CS	Čeština	HE	עברית	NO	Norsk	SV	Svenska
DA	Dansk	HR	Hrvatski	PL	Polski	UK	Українська
DE	Deutsch	HU	Magyar	PT	Português		
EN	English	IT	Italiano	RU	Русский		
ES	Español	JA	日本語	RO	Română		
ET	Eesti	LV	Latviešu	SA	العربية		
FI	Suomi	LT	Lietuvių	SK	Slovenčina		

Pos.	Part number	Description	Valid from → until	Qty
EN	SP1555731	User Manual Proximity Head Remote DSL-105 English		1
DE	SP1555732	User Manual Proximity Head Remote DSL-105 German		1
FR	SP1555733	User Manual Proximity Head Remote DSL-105 French		1
DA	SP1555839	User Manual Proximity Head Remote DSL-105 Danish		1
ES	SP1555840	User Manual Proximity Head Remote DSL-105 Spanish		1
FI	SP1555841	User Manual Proximity Head Remote DSL-105 Finnish		1
IT	SP1555842	User Manual Proximity Head Remote DSL-105 Italian		1
NL	SP1555843	User Manual Proximity Head Remote DSL-105 Dutch		1
NO	SP1555844	User Manual Proximity Head Remote DSL-105 Norwegian		1
PT	SP1555845	User Manual Proximity Head Remote DSL-105 Portuguese		1
SV	SP1555846	User Manual Proximity Head Remote DSL-105 Swedish		1

User Manuals ACS Handprogrammer





languages.iso

BG	Български	FR	Français	NL	Nederlands	SL	Slovenščina
CS	Čeština	HE	עברית	NO	Norsk	SV	Svenska
DA	Dansk	HR	Hrvatski	PL	Polski	UK	Українська
DE	Deutsch	HU	Magyar	PT	Português		
EN	English	IT	Italiano	RU	Русский		
ES	Español	JA	日本語	RO	Română		
ET	Eesti	LV	Latviešu	SA	العربية		
FI	Suomi	LT	Lietuvių	SK	Slovenčina		

Pos.	Part number	Description	Valid from → until	Qty
DE	1419316	User Manual Handheld Programmer German		1
EN	1419317	User Manual Handheld Programmer English		1

Change list Recaro 24V

1420465X.iso

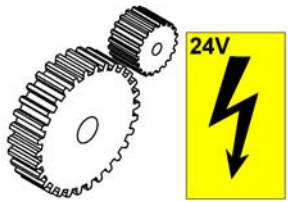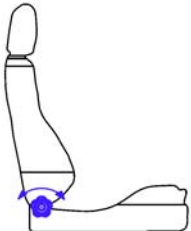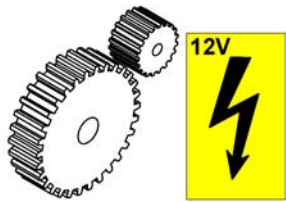				
Storm4	O	O	P	P
G50	O	O	P	special
FDX	O	O	P	special
Typhoon	O	P	O	O
Typhoon II	O	P	O	O
Zephyr	O	P	O	O
Tornado	O	P	O	O
Storm3	O	P	O	O


Pos.	Part number	Description	Valid from → until	Qty
-	-	For further information please contact your customer service.		1

Storm4

Pos.	Part number	Description	Valid from → until	Qty
-		Chassis Fittings		1
-		Chassis Fittings (with Ultra Low Maxx Seat)		1
-		Front swingarm		1
-		Front Swing Arm Storm4 X-Plore		1
-		Motor Swingarm => 04.01.11		1
-		Motor Swingarm SSD		1
-		Motor Swingarm SSD (with Ultra Low Max Seat)		1
-		Motor Swingarm SSD, Anti-Tippers flip up		1
-		Rear suspension => 03.08.12		1
-		Rear suspension 04.08.12 =>		1
-		Upper suspension, Storm4 X-Plore		1
-		Transport fixation => 03.08.12		1
-		Transport fixation 04.08.12 =>		1
-		Shrouds		1
-		Shrouds (with Ultra Low Maxx Seat)		1
-		Shroud spring bracket/rear light		1

Chassis Fittings


1527810X.iso

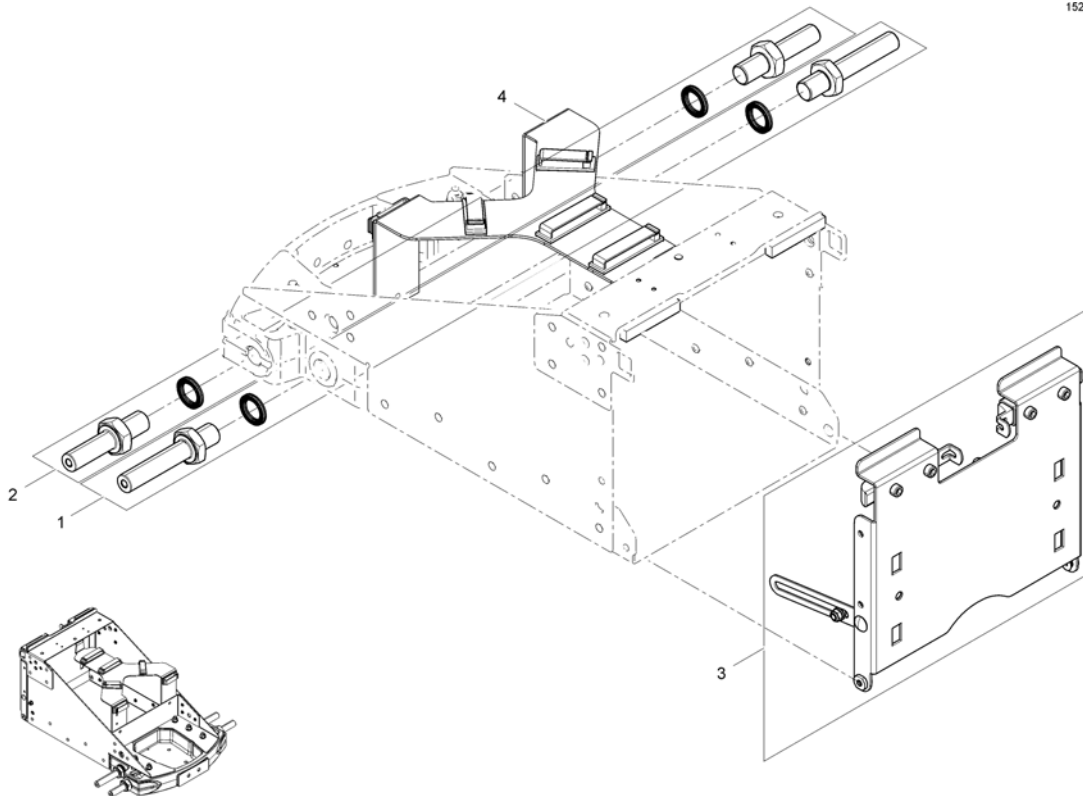

Pos.	Part number	Description	Valid from → until	Qty
1	SP1530317	Main Bolt		1
2	SP1540337	Main Bolt (Storm4 X-Plore)		1
3	SP1526347	Batterydoor		1
4	SP1524458	Cable support, cpl.		1
5	SP1561820	Plug for M8		3

Chassis Fittings (with Ultra Low Maxx Seat)



1602106.kim

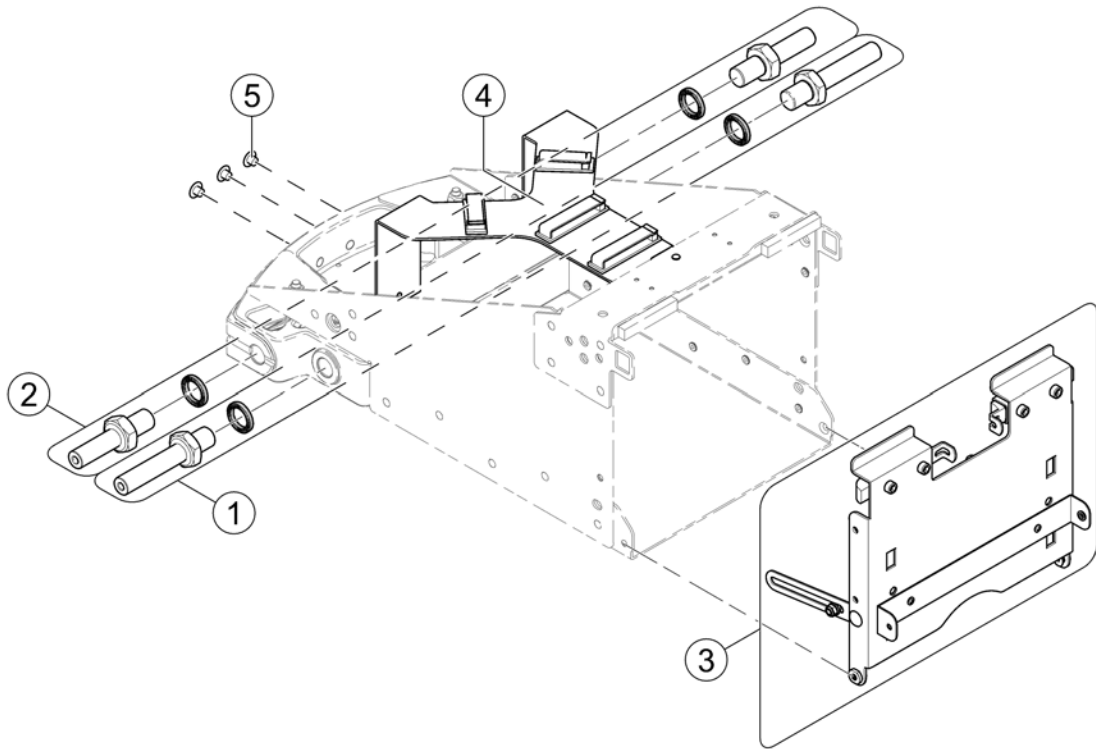

Pos.	Part number	Description	Valid from → until	Qty
1	SP1530317	Main Bolt		1
2	SP1540337	Main Bolt (Storm4 X-Plore)		1
3	SP1543040	Battery door		1
4	SP1533336	Cable support, cpl.		1
5	SP1561820	Plug for M8		3

Front swingarm


1534668-00

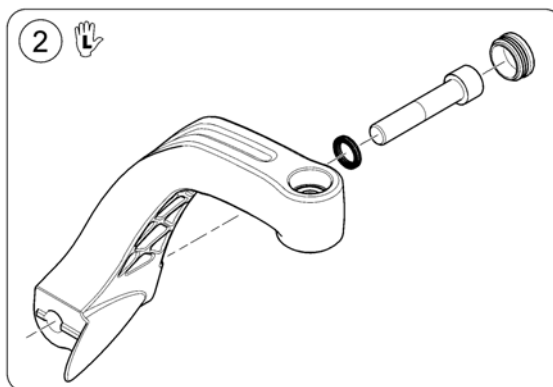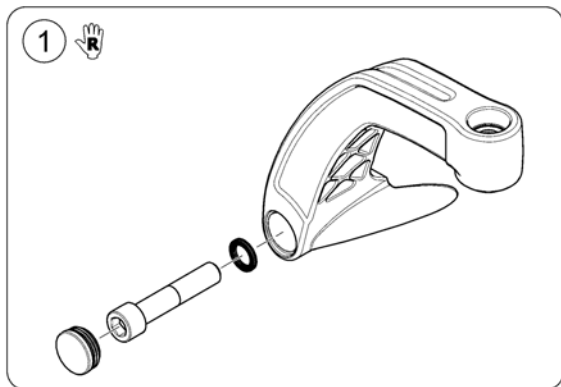

Pos.	Part number	Description	Valid from → until	Qty
1	SP1527905	Swingarm RH		1
2	SP1527904	Swingarm LH		1

Front Swing Arm Storm4 X-Plore


1537881X.dwg

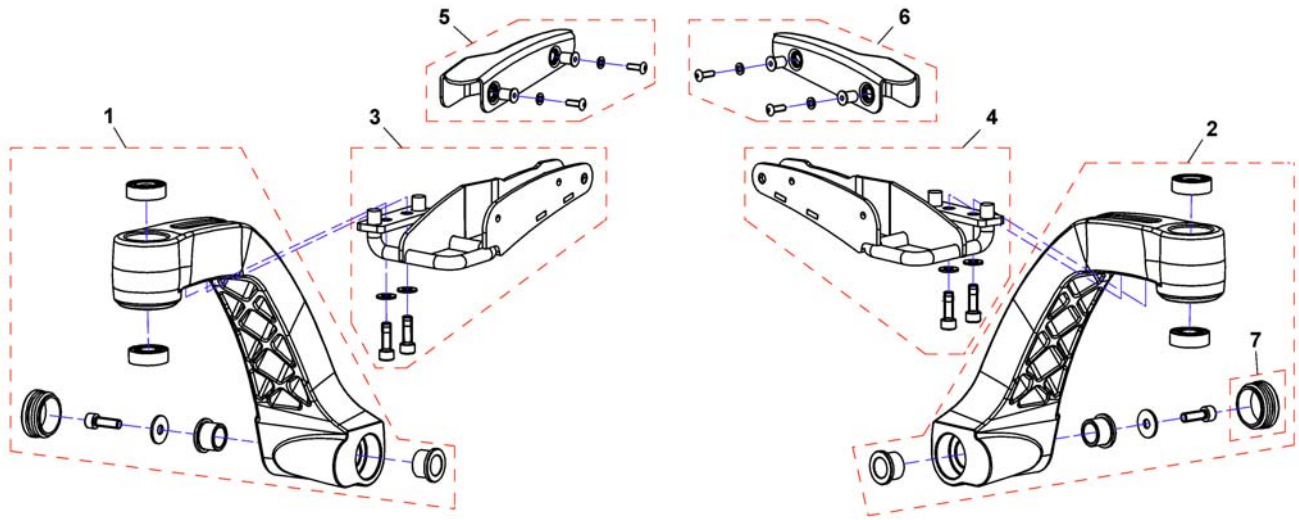

Pos.	Part number	Description	Valid from → until	Qty
1	SP1540359	Swingarm, Storm4 X-Plore, RH		1
2	SP1540360	Swingarm, Storm4 X-Plore, LH		1
3	SP1540361	Suspension bracket, Storm4 X-Plore, RH		1
4	SP1541357	Suspension bracket, Storm4 X-Plore, LH		1
5	SP1541358	Cover suspension bracket, Storm4 X-Plore, RH		1
6	SP1541359	Cover suspension bracket, Storm4 X-Plore, LH		1
7	SPL01043	Slider		1

Motor Swingarm => 04.01.11


1581873.iso

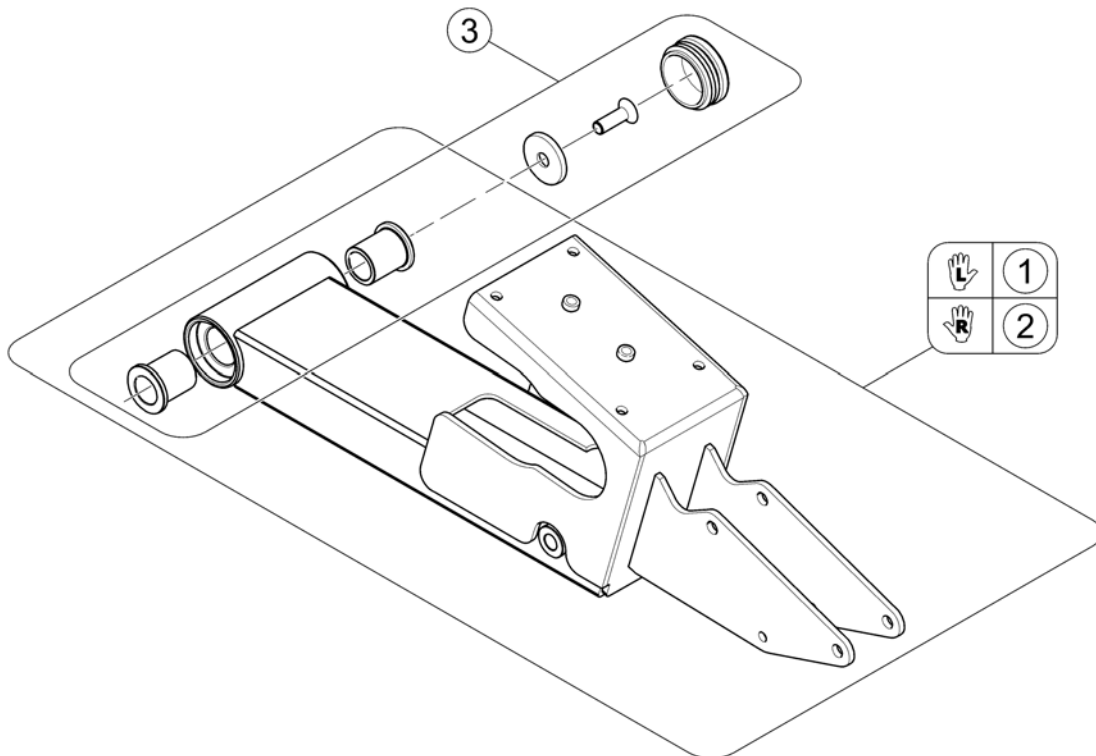

Pos.	Part number	Description	Valid from → until	Qty
1	SP1541800	Swingarm LH		1
2	SP1541799	Swingarm RH		1
3	SP1530581	Bush Bearing-Kit		1

Motor Swingarm SSD


1533674X.dwg

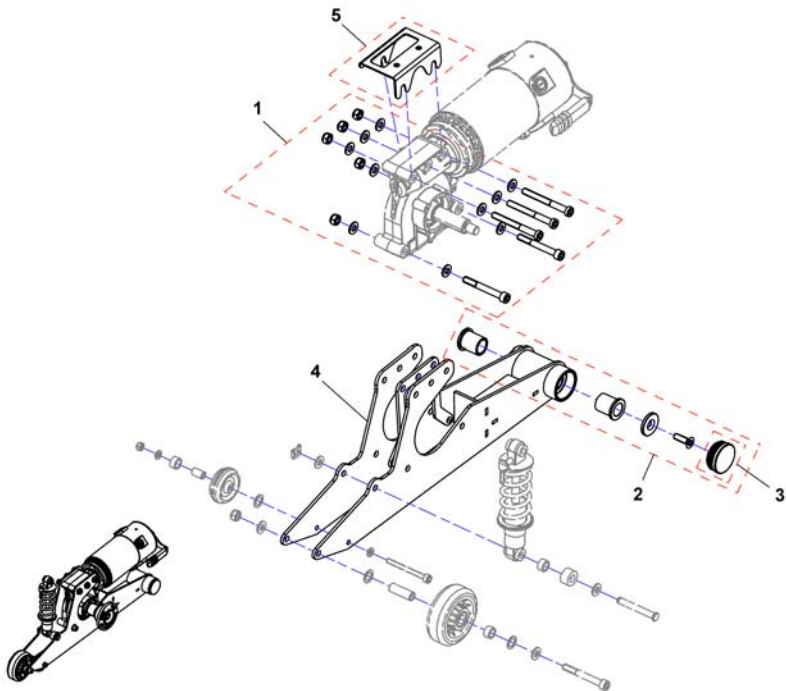

Pos.	Part number	Description	Valid from → until	Qty
1	SP1541363	Mounting Hardware, Motor Swing Arm		1
2	SP1541367	Bush Bearing-Kit		1
3	SPL01043	Slider		1
4	SP1541761	Swingarm RH		1
4	SP1541762	Swingarm LH		1
5	SP1533974	Bridge		1


Motor Swingarm SSD (with Ultra Low Max Seat)

1602356.iss

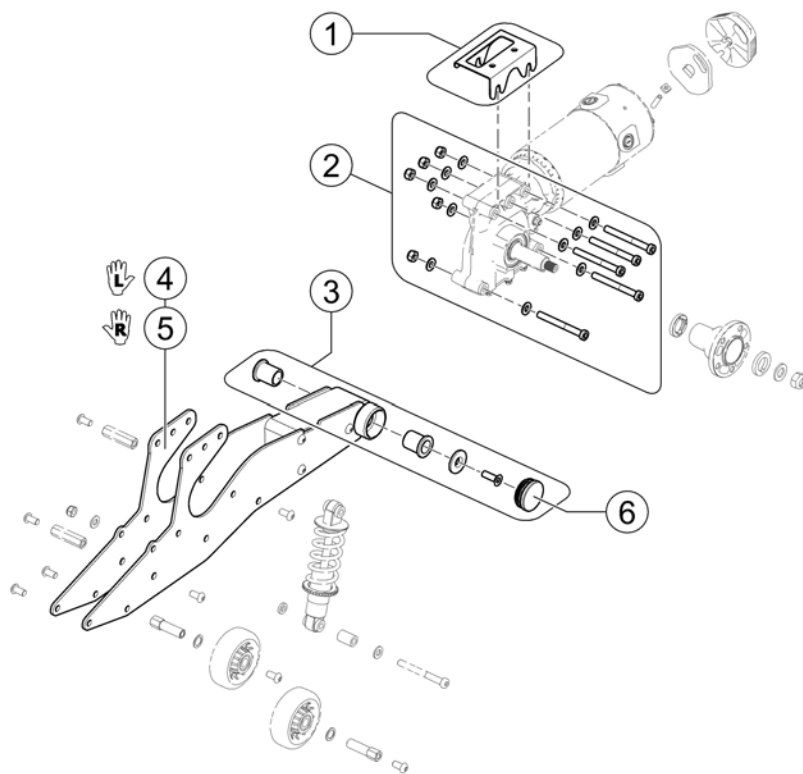

Pos.	Part number	Description	Valid from → until	Qty
1	SP1533974	Bridge		1
2	SP1541363	Mounting Hardware, Motor Swing Arm		1
3	SP1541367	Bush Bearing-Kit		1
4	SP1587251	Swingarm LH		1
5	SP1587252	Swingarm RH		1
6	SPL01043	Slider		1


Motor Swingarm SSD, Anti-Tippers flip up

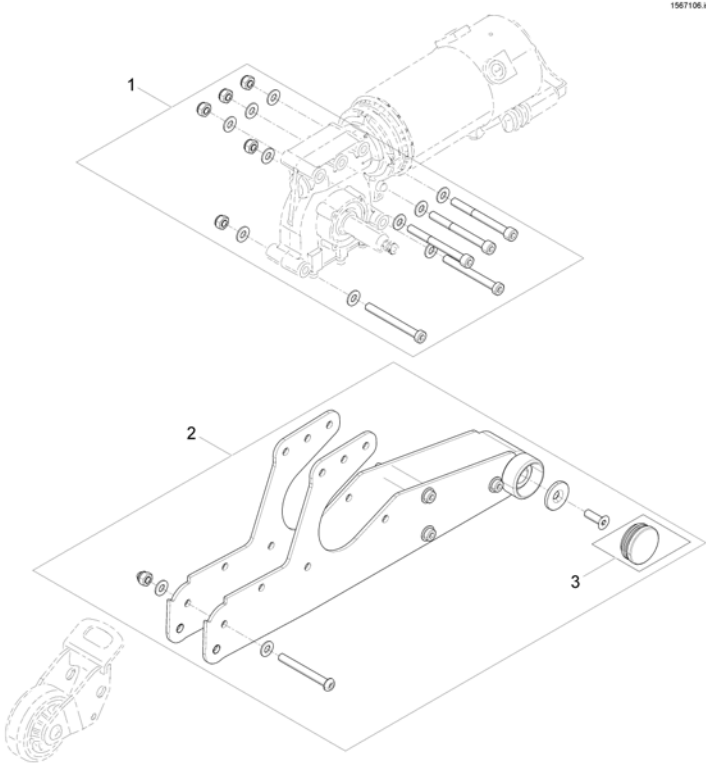

Pos.	Part number	Description	Valid from → until	Qty
1	SP1541363	Mounting Hardware, Motor Swing Arm		1
2	SP1552980	Swingarm LH		1
3	SP1552979	Swingarm RH		1
4	SPL01043	Slider		1


Rear suspension => 03.08.12

152896X.00

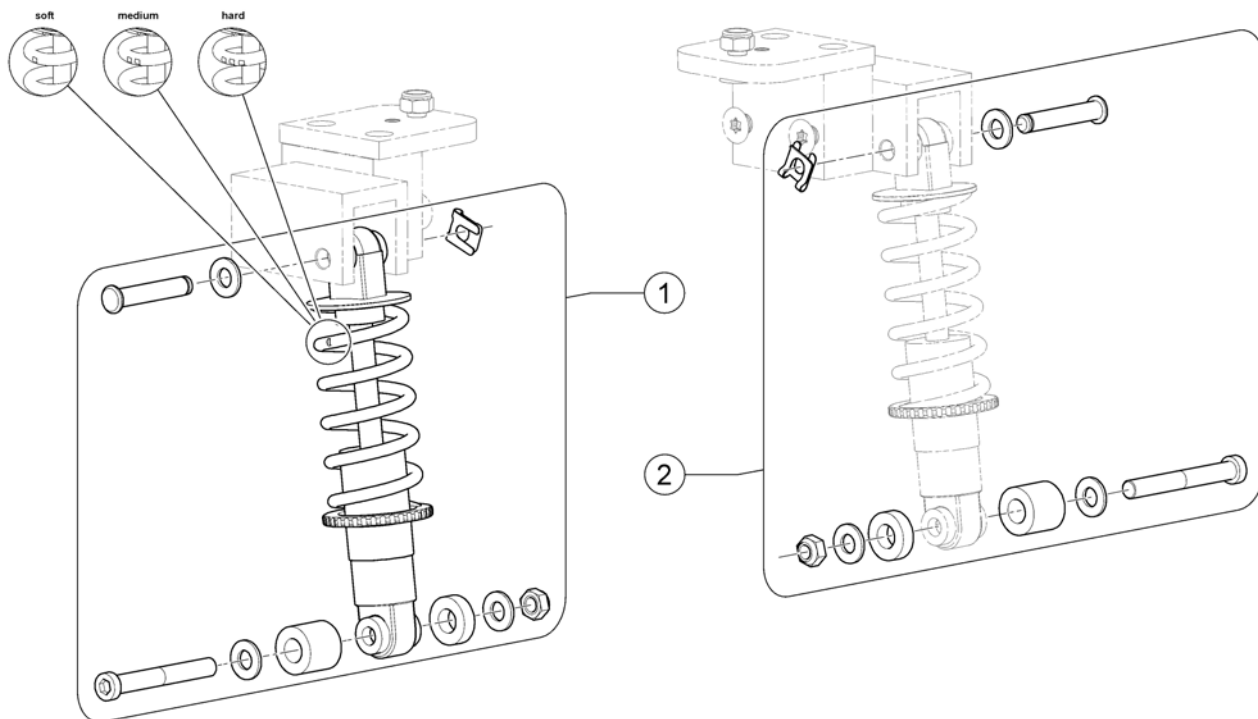

Pos.	Part number	Description	Valid from → until	Qty
1	SP1530318	Shock absorber with mounting hardware - soft		1
1	SP1530337	Shock absorber with mounting hardware - medium		1
1	SP1530338	Shock absorber with mounting hardware - hard		1
1	SP1540343	Shock absorber, Storm4 X-Plore, cpl.		1
2	SP1530444	Mounting-Kit		1


Rear suspension 04.08.12 =>

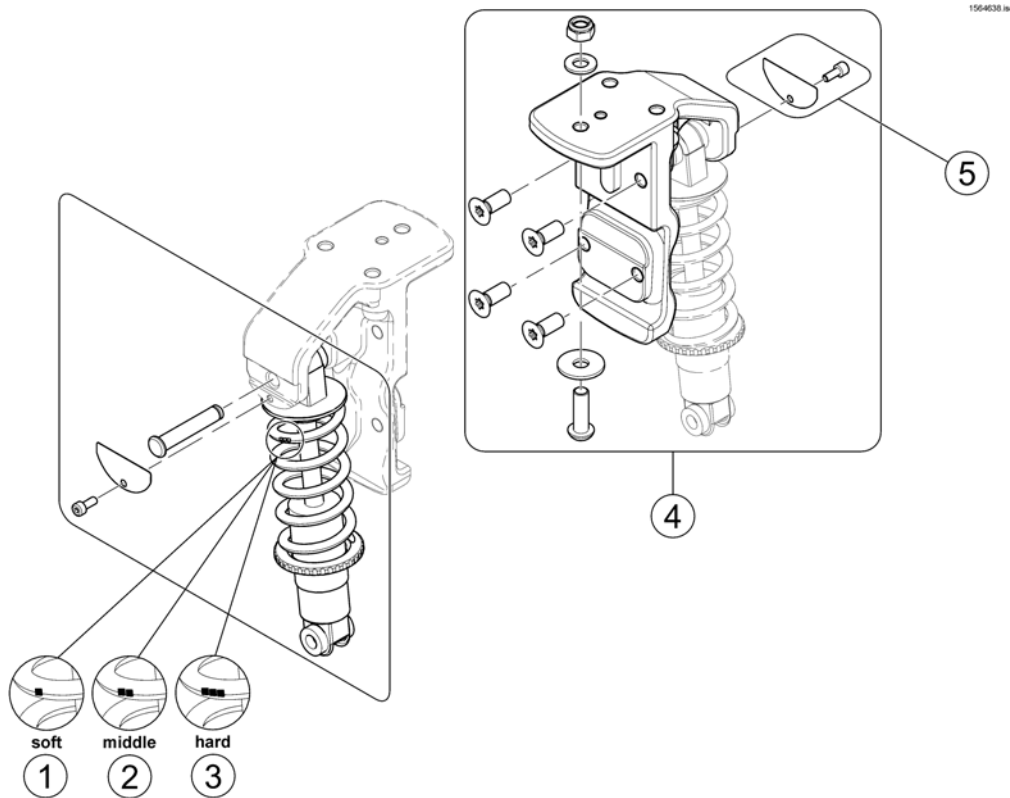

Pos.	Part number	Description	Valid from → until	Qty
1	SP1564530	Shock absorber, soft, cpl.		1
2	SP1564531	Shock absorber, middle, cpl.		1
3	SP1564532	Shock absorber, hard, cpl.		1
-	SP1564636	Shock absorber, Storm4 X-Plore, cpl.		1
4	SP1528950	Shock absorber support, cpl.		1
5	SP1564637	Mounting-Kit		1

Upper suspension, Storm4 X-Plore


1540327X.iso

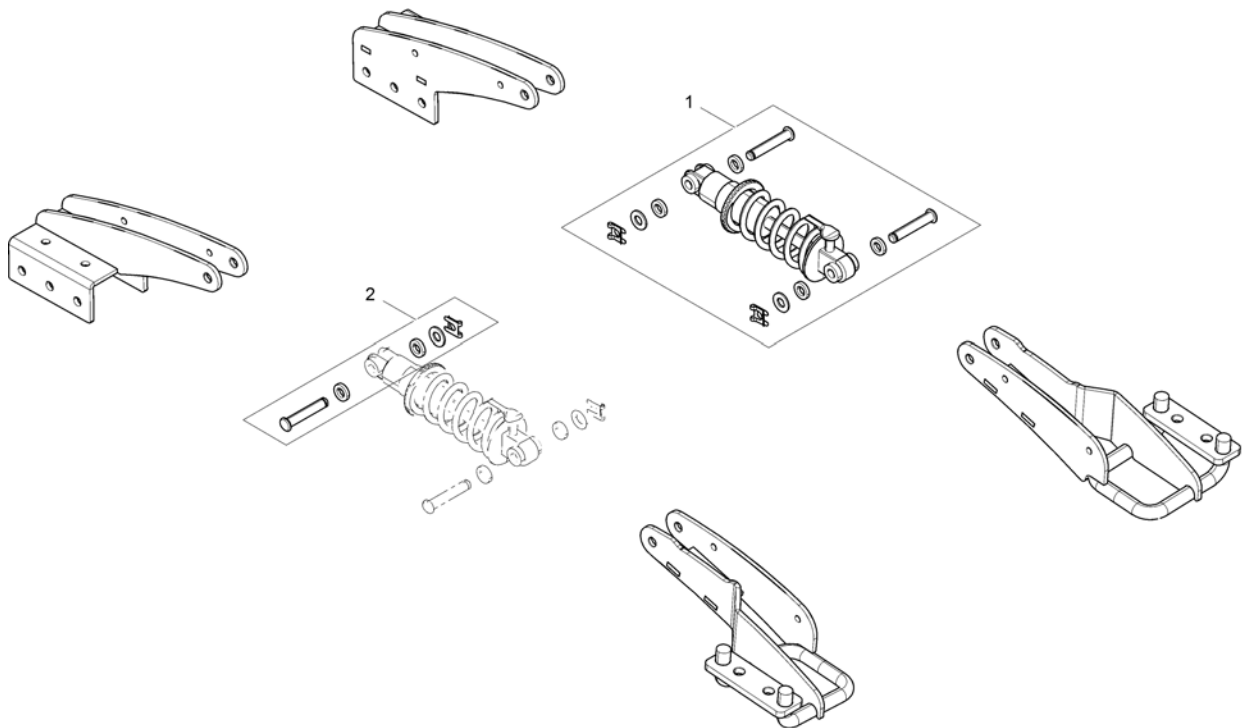

Pos.	Part number	Description	Valid from → until	Qty
1	SP1540348	Shock absorber front swing arm, soft, cpl.		1
1	SP1540349	Shock absorber front swing arm, hard, cpl.		1
2	SP1540350	Mountingkit		1

Transport fixation => 03.08.12


156419.dwg

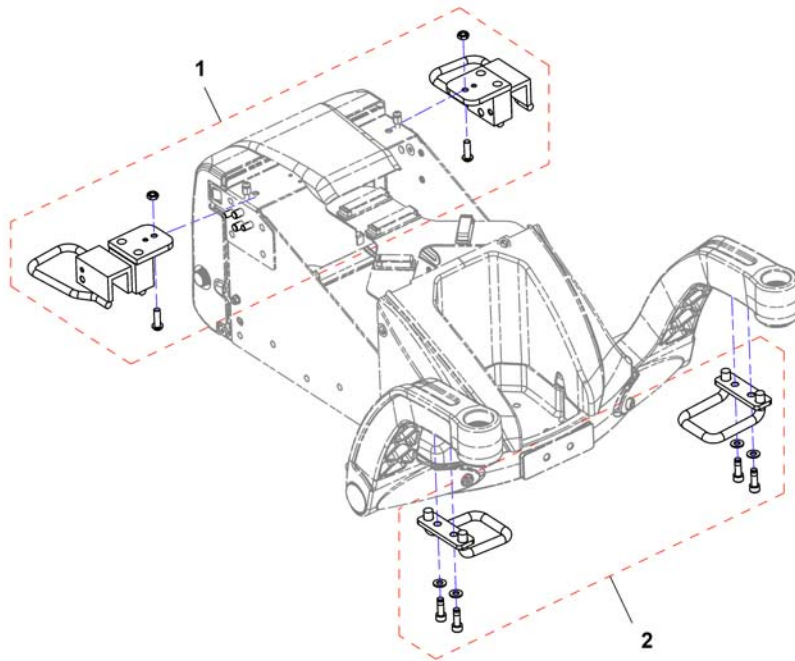

Pos.	Part number	Description	Valid from → until	Qty
1	SP1563104	Transport fixation rear <i>not for passenger transportation</i>		1
2	SP1528974	Transport fixation, front <i>not for passenger transportation</i>		1


Transport fixation 04.08.12 =>

1564528.iso

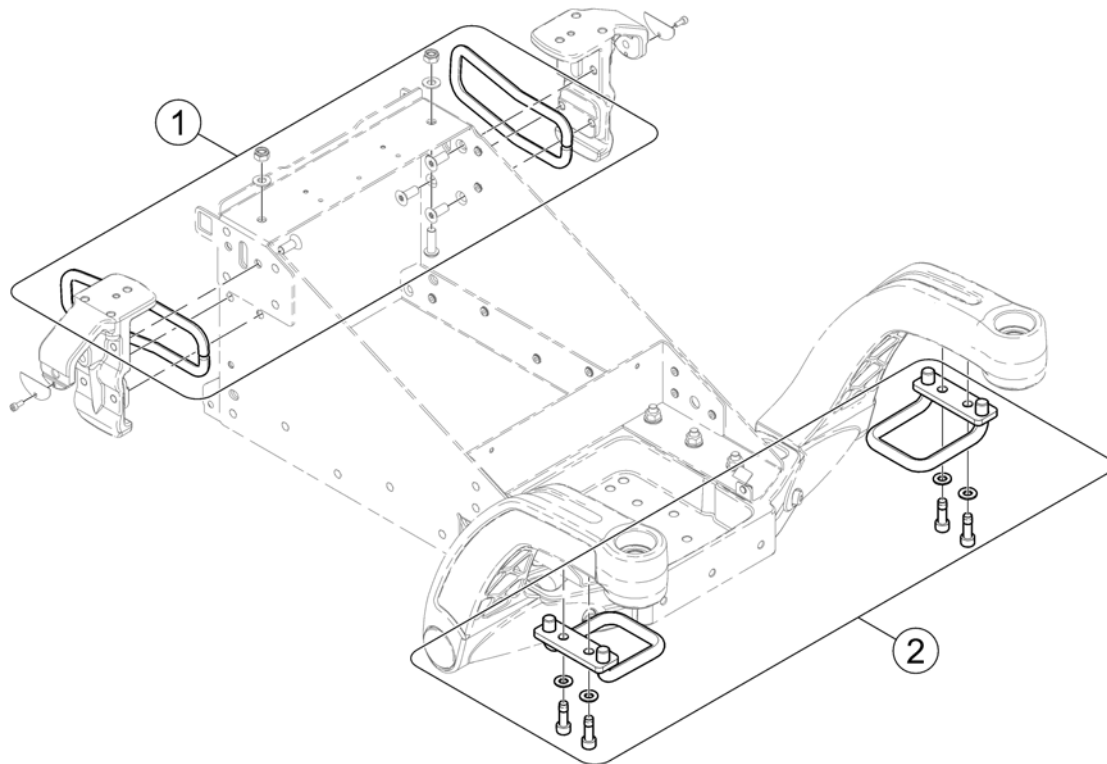

Pos.	Part number	Description	Valid from → until	Qty
1	SP1528951	Transport fixation, rear <i>not for passenger transportation</i>		1
2	SP1528974	Transport fixation, front <i>not for passenger transportation</i>		1

Shrouds

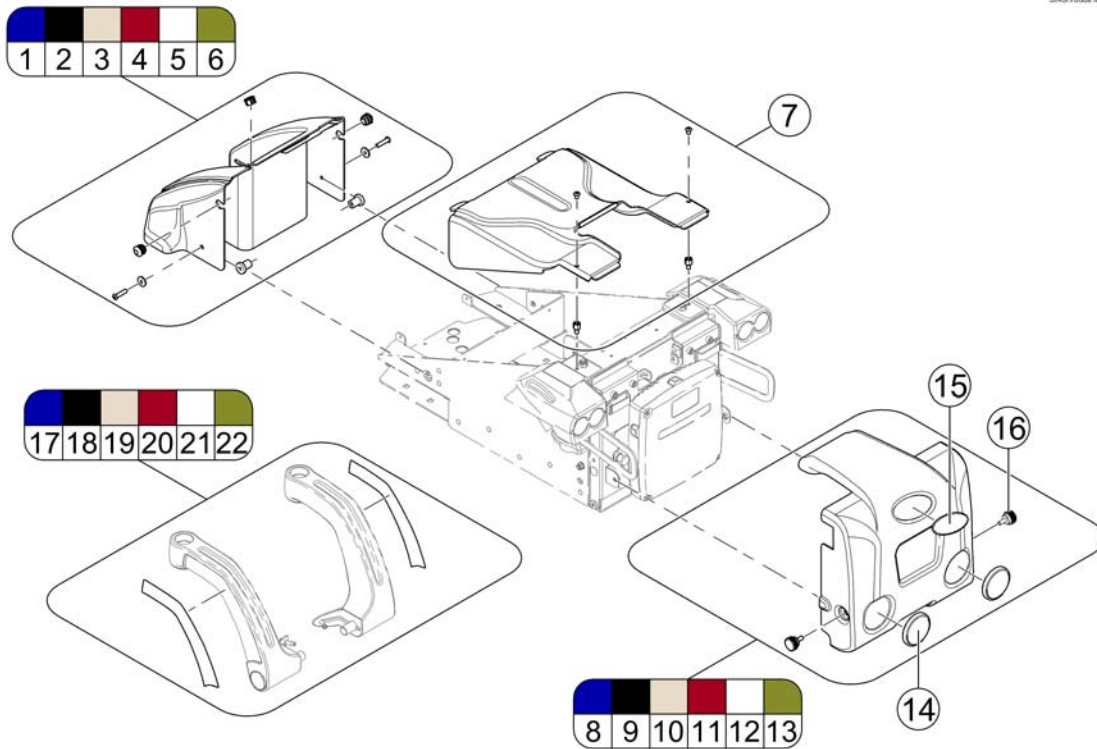


Pos.	Part number	Description	Valid from → until	Qty
	SP1553166	Touch-up paint - Blue (RAL 5026)		1
	SP1553148	Touch-up paint, RAL 9005 Black		1
	SP1553167	Touch-up paint - Champagne		1
	SP1553169	Touch-up paint - Red		1
	SP1553168	Touch-up paint - White (RAL 9016)		1
	SP1553210	Touch-up paint - Green (RAL 6003)		1
	SP1553211	Touch-up paint - White aluminium (RAL 9006)		1
1	SP1530339	Front shroud, cpl., Blue (RAL 5026)	→ 2012-04-12	1
1	SP1559801	Front shroud, cpl. - blue (RAL 5026)	2012-04-13 →	1
2	SP1530340	Front shroud, cpl., Black (RAL 9005)	→ 2012-04-09	1
2	SP1559802	Front shroud, cpl. - black (RAL 9005)	2012-04-10 →	1
3	SP1530341	Front shroud, cpl., Champagne	→ 2012-04-12	1
3	SP1559803	Front shroud, cpl. - champagne	2012-04-13 →	1
4	SP1530342	Front shroud, cpl., Red	→ 2012-04-12	1
4	SP1559804	Front shroud, cpl. - red	2012-04-13 →	1
5	SP1530343	Front shroud, cpl., White (RAL 9016)	→ 2012-04-12	1
5	SP1559805	Front shroud, cpl. - white (RAL 9016)	2012-04-13 →	1
6	SP1530344	Front shroud, cpl., Green (RAL 6003)	→ 2012-04-12	1
6	SP1559806	Front shroud, cpl. - green (RAL 6003)	2012-04-13 →	1
7	SP1530345	Middle Shroud - white aluminium (RAL 9006)		1
8	SP1530346	Rear shroud, cpl., Blue (RAL 5026)		1
9	SP1530347	Rear shroud, cpl., Black (RAL 9005)		1
10	SP1530348	Rear shroud, cpl., Champagne		1
11	SP1530427	Rear shroud, cpl., Red		1
12	SP1530428	Rear shroud, cpl., White (RAL 9016)		1
13	SP1530429	Rear shroud, cpl., Green (RAL 6003)		1
14	1419274	Reflector round, red		1
15	0572-06266	Invacare logo		1
16	1440769	Hand screw M6x15		1
17	SP1522428	Decal-Kit Blue (RAL 5026)		1
18	SP1523651	Decal-Kit Black (RAL 9005)		1

Shrouds

Pos.	Part number	Description	Valid from → until	Qty
19	SP1529145	Decal-Kit Champagne		1
20	SP1529393	Decal-Kit Red (RAL 3007)		1
21	SP1529395	Decal-Kit White (RAL 9016)		1
22	SP1529397	Decal-Kit Green (RAL 6003)		1


Shrouds (with Ultra Low Maxx Seat)

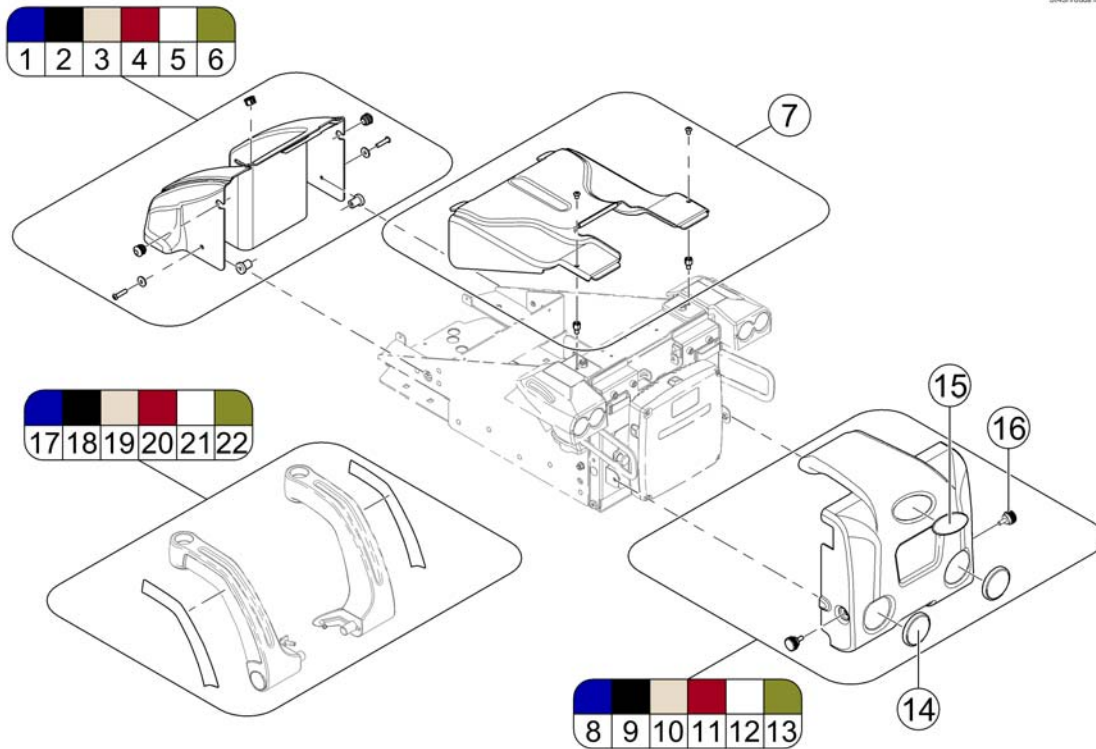

Pos.	Part number	Description	Valid from → until	Qty
	SP1553166	Touch-up paint - Blue (RAL 5026)		1
	SP1553148	Touch-up paint, RAL 9005 Black		1
	SP1553167	Touch-up paint - Champagne		1
	SP1553169	Touch-up paint - Red		1
	SP1553168	Touch-up paint - White (RAL 9016)		1
	SP1553210	Touch-up paint - Green (RAL 6003)		1
	SP1553211	Touch-up paint - White aluminium (RAL 9006)		1
1	SP1559801	Front shroud, cpl. - blue (RAL 5026)		1
2	SP1559802	Front shroud, cpl. - black (RAL 9005)		1
3	SP1559803	Front shroud, cpl. - champagne		1
4	SP1559804	Front shroud, cpl. - red		1
5	SP1559805	Front shroud, cpl. - white (RAL 9016)		1
6	SP1559806	Front shroud, cpl. - green (RAL 6003)		1
7	SP1546754	Middle shroud - White (RAL 9006)		1
8	SP1530346	Rear shroud, cpl., Blue (RAL 5026)		1
9	SP1530347	Rear shroud, cpl., Black (RAL 9005)		1
10	SP1530348	Rear shroud, cpl., Champagne		1
11	SP1530427	Rear shroud, cpl., Red		1
12	SP1530428	Rear shroud, cpl., White (RAL 9016)		1
13	SP1530429	Rear shroud, cpl., Green (RAL 6003)		1
14	1419274	Reflector round, red		1
15	0572-06266	Invacare logo		1
16	1440769	Hand screw M6x15		1
17	SP1522428	Decal-Kit Blue (RAL 5026)		1
18	SP1523651	Decal-Kit Black (RAL 9005)		1
19	SP1529145	Decal-Kit Champagne		1
20	SP1529393	Decal-Kit Red (RAL 3007)		1
21	SP1529395	Decal-Kit White (RAL 9016)		1
22	SP1529397	Decal-Kit Green (RAL 6003)		1

Shroud spring bracket/rear light

1528810X.iso

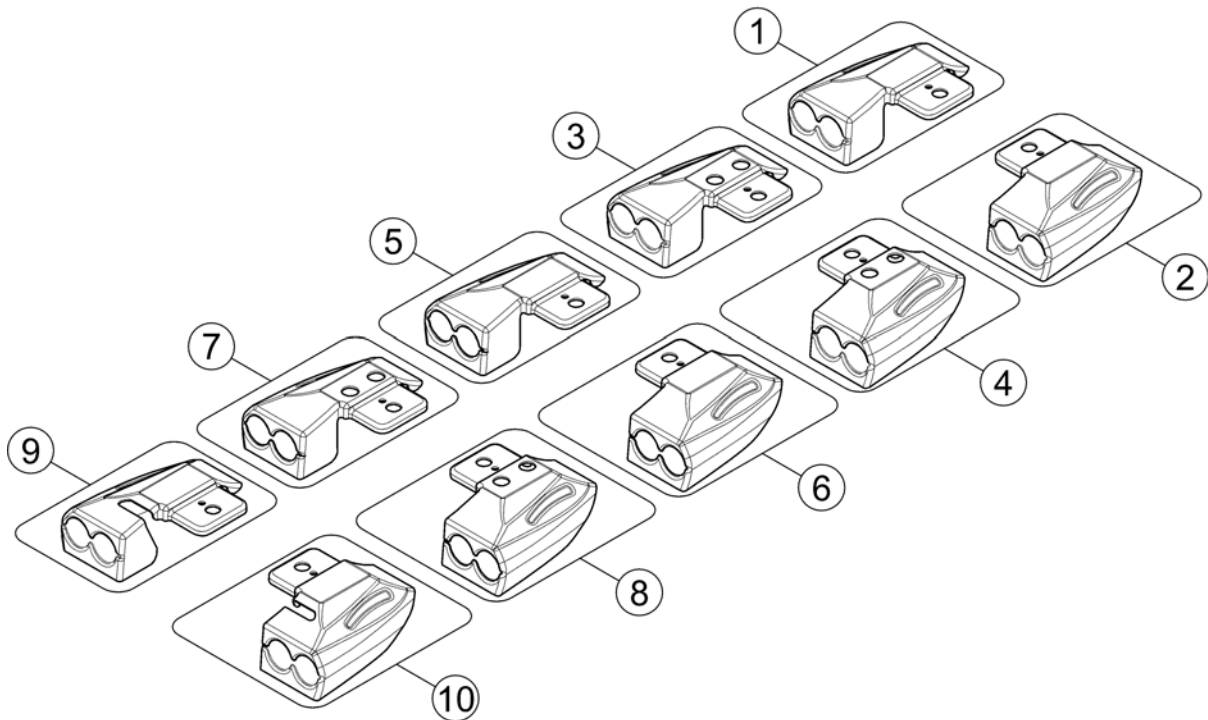


Pos.	Part number	Description	Valid from → until	Qty
1	SP1528810	Shroud for spring bracket without light LH		1
2	SP1528811	Shroud for spring bracket without light RH		1
3	SP1552927	Shroud for spring bracket, with baggage carrier, without light, LH		1
4	SP1552928	Shroud for spring bracket, with baggage carrier, without light, RH		1
5	SP1527836	Shroud for spring bracket for LED-rearlight LH		1
6	SP1527835	Shroud for spring bracket for LED-rearlight RH		1
7	SP1552929	Shroud for spring bracket, with baggage carrier, for LED-rearlight, LH		1
8	SP1552930	Shroud for spring bracket, with baggage carrier, for LED-rearlight, RH		1
9	SP1528627	Shroud for spring bracket for rearlight Germany, LH		1
10	SP1528626	Shroud for spring bracket for rearlight Germany, RH		1

Storm4 TTPlus

Pos.	Part number	Description	Valid from → until	Qty
-		Chassis Fittings		1
-		Front swingarm		1
-		Front Swing Arm Storm4 X-Plore		1
-		Motor Swingarm (GGB)		0
-		Suspension		0
-		Upper suspension, Storm4 X-Plore		1
-		Transport Fixation		0
-		Shrouds		0
-		Shrouds Spring Bracket		0

Motor Swingarm (GGB)


1581904.iso

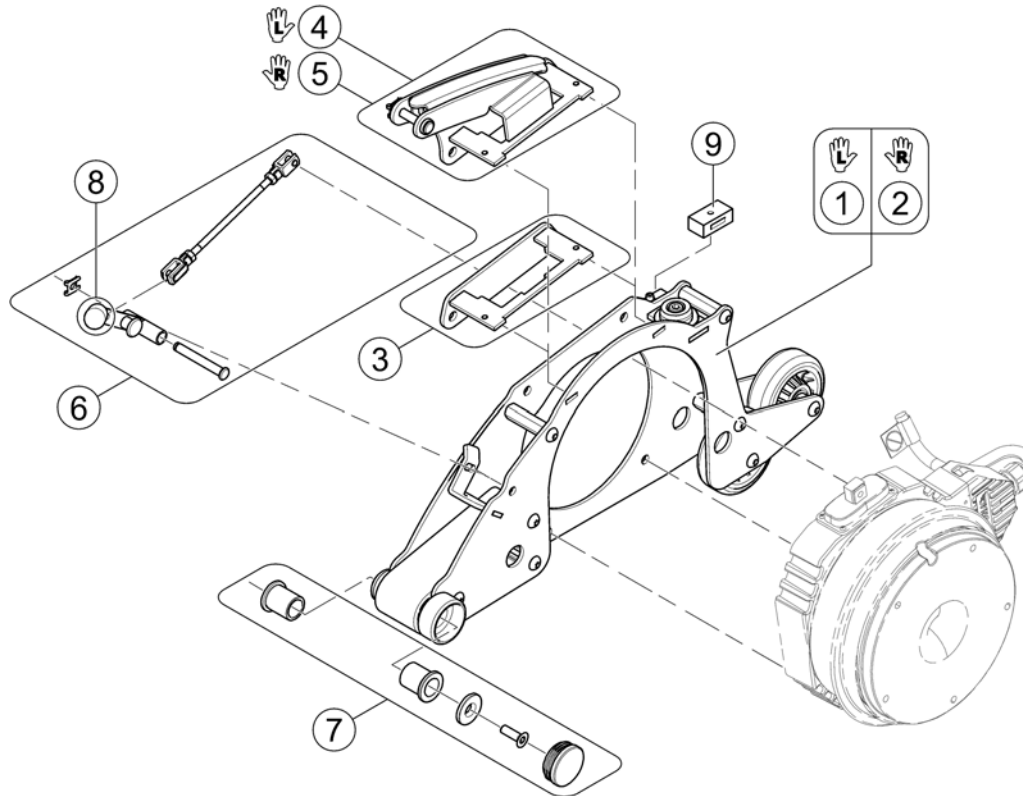

Pos.	Part number	Description	Valid from → until	Qty
1	SP1582232	Motor swingarm (GGB2), cpl. LH		1
2	SP1582233	Motor swingarm (GGB2), cpl. RH		1
3	SP1574130	Bridge (GGB2), cpl.		1
4	SP1576525	Shockabsorber Support X-Plore (GGB2), LH		1
5	SP1576526	Shockabsorber Support X-Plore (GGB2), RH		1
6	SP1582218	Brake Lever Kit (GGB2), cpl.		1
7	SP1541363	Mounting Hardware, Motor Swing Arm		1
8	5007791	KNOB; BLACK		1
9	SP1574283	Rubber Stop		1

Suspension


1581876 iso

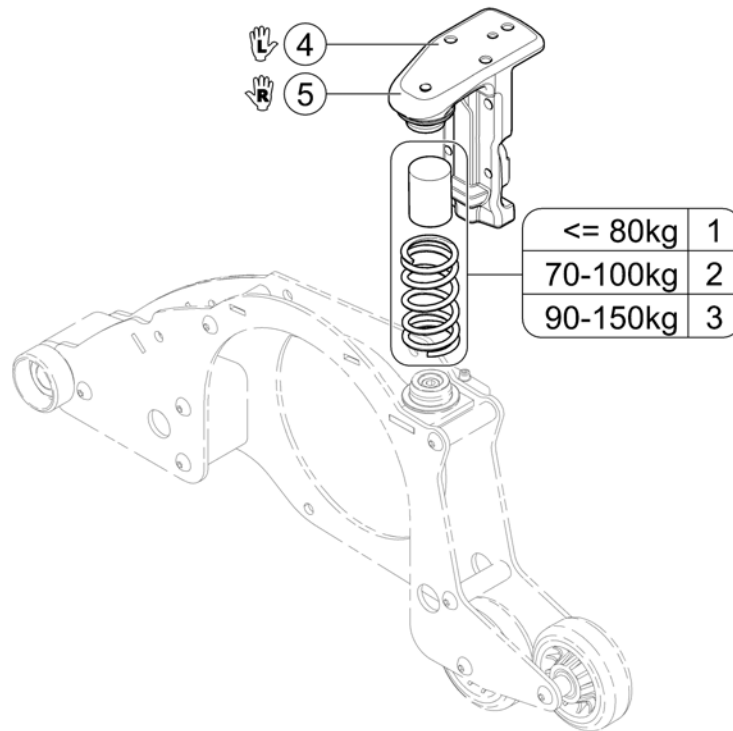

Pos.	Part number	Description	Valid from → until	Qty
1	SP1582237	Spring Element <= 80kg		1
2	SP1582363	Spring Element 70-100kg		1
3	SP1582364	Spring Element 90-150kg		1
4	SP1577234	Suspension Support, LH		1
5	SP1574142	Suspension Support, RH		1

Transport Fixation


1564528.iso

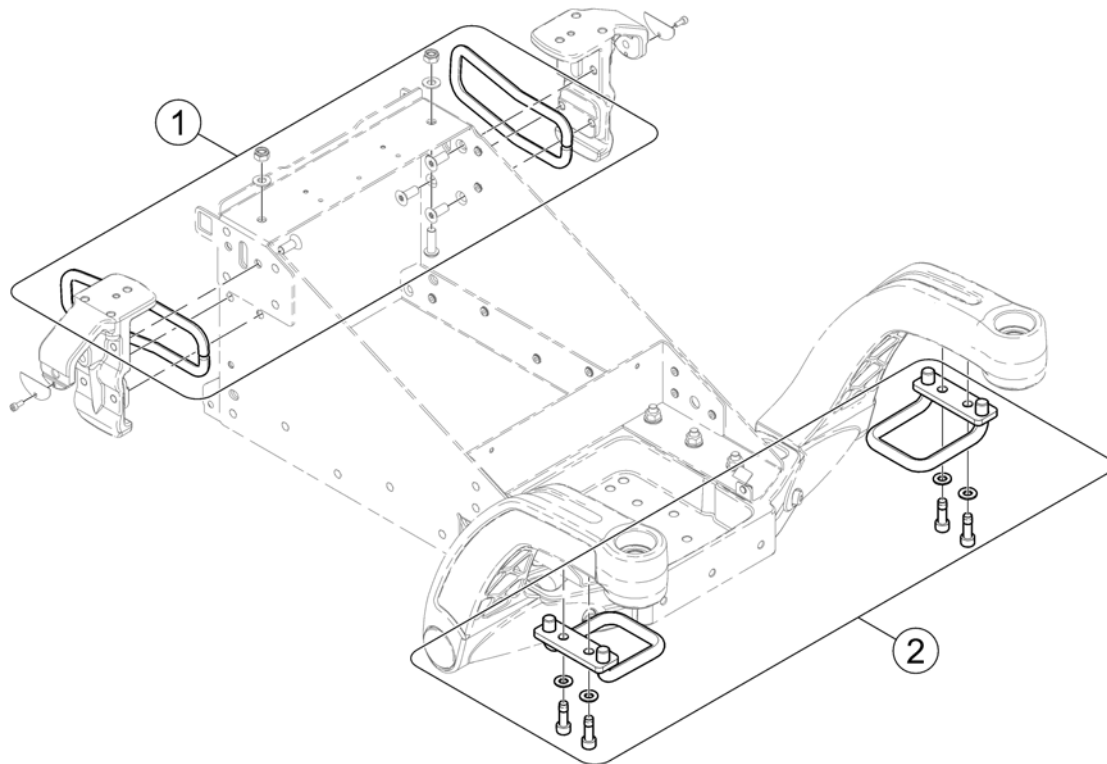

Pos.	Part number	Description	Valid from → until	Qty
1	SP1582234	Transport fixation, rear		1
2	SP1528974	Transport fixation, front		1

Shrouds


1581874.000

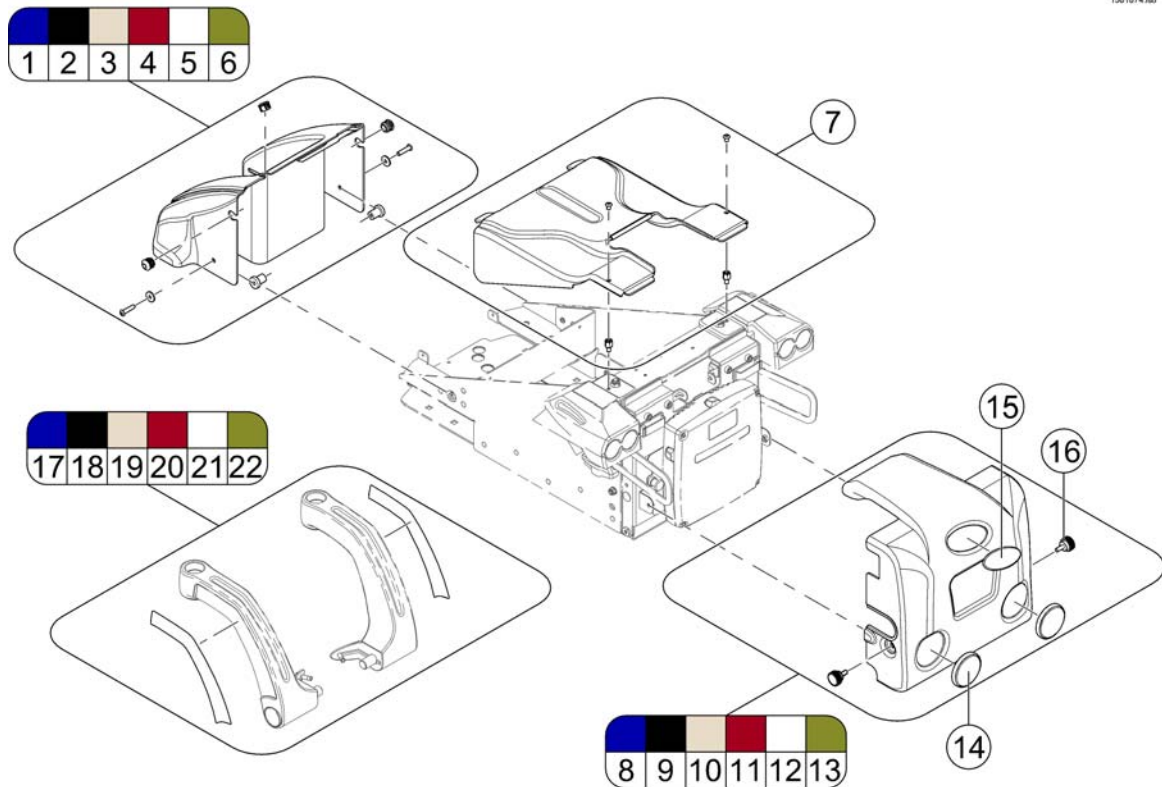



Pos.	Part number	Description	Valid from → until	Qty
	SP1553166	Touch-up paint - Blue (RAL 5026)		1
	SP1553148	Touch-up paint, RAL 9005 Black		1
	SP1553167	Touch-up paint - Champagne		1
	SP1553169	Touch-up paint - Red		1
	SP1553168	Touch-up paint - White (RAL 9016)		1
	SP1553210	Touch-up paint - Green (RAL 6003)		1
	SP1553211	Touch-up paint - White aluminium (RAL 9006)		1
1	SP1559801	Front shroud, cpl. - blue (RAL 5026)		1
2	SP1559802	Front shroud, cpl. - black (RAL 9005)		1
3	SP1559803	Front shroud, cpl. - champagne		1
4	SP1559804	Front shroud, cpl. - red		1
5	SP1559805	Front shroud, cpl. - white (RAL 9016)		1
6	SP1559806	Front shroud, cpl. - green (RAL 6003)		1
7	SP1530345	Middle Shroud - white aluminium (RAL 9006)		1
8	SP1581950	Rear Shroud, cpl. - blue (RAL 5026)		1
9	SP1581951	Rear Shroud, cpl. - black (RAL 9005)		1
10	SP1581952	Rear Shroud, cpl. - champagne		1
11	SP1582000	Rear Shroud, cpl. - transparent cherry red		1
12	SP1582001	Rear Shroud, cpl. - white (RAL 9016)		1
13	SP1582002	Rear Shroud cpl. - olive green (RAL 6003)		1
14	1419274	Reflector round, red		1
15	0572-06266	Invacare logo		1
16	1440769	Hand screw M6x15		1
17	SP1522428	Decal-Kit Blue (RAL 5026)		1
18	SP1523651	Decal-Kit Black (RAL 9005)		1
19	SP1529145	Decal-Kit Champagne		1
20	SP1529393	Decal-Kit Red (RAL 3007)		1
21	SP1529395	Decal-Kit White (RAL 9016)		1
22	SP1529397	Decal-Kit Green (RAL 6003)		1

Shrouds Spring Bracket

1528810X.iso

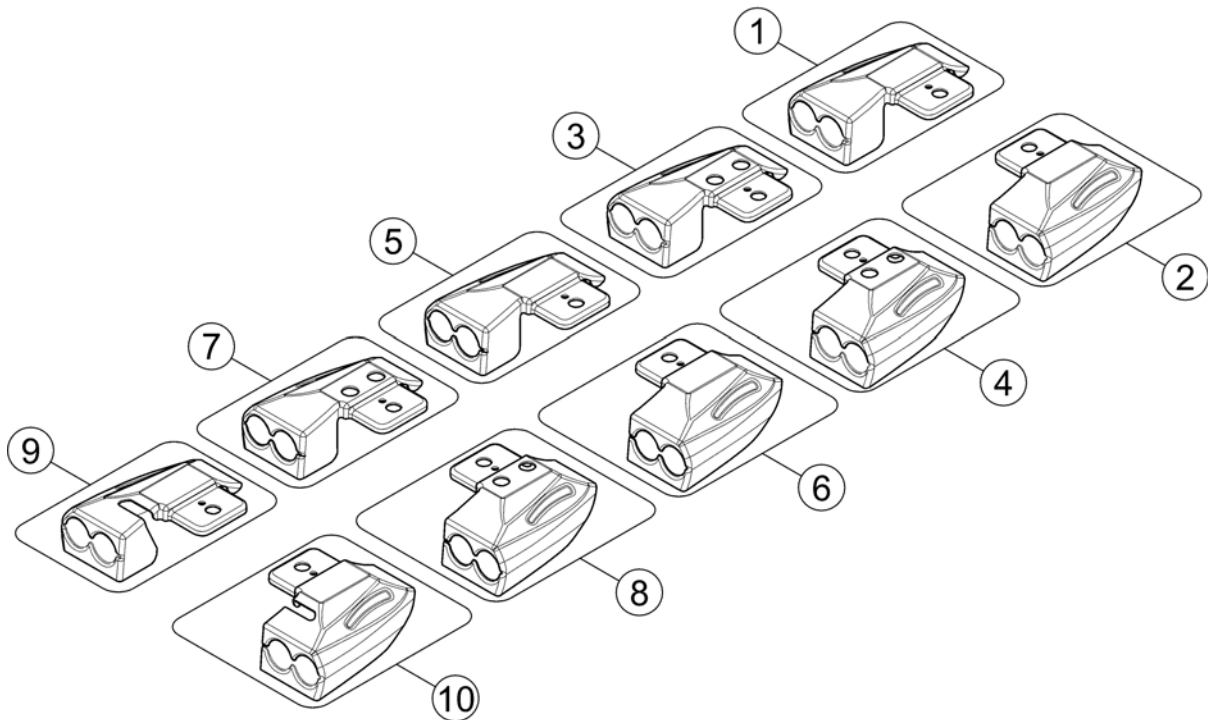


Pos.	Part number	Description	Valid from → until	Qty
1	SP1577034	Shroud for spring bracket without light LH GGB2		1
2	SP1577035	Shroud for spring bracket without light RH GGB2		1
3	SP1577056	Shroud for spring bracket with luggage-rack, w/o light LH GGB2		1
4	SP1577057	Shroud for spring bracket with luggage-rack, w/o light RH GGB2		1
5	SP1574138	Shroud for spring bracket with LED light LH GGB2		1
6	SP1574139	Shroud for spring bracket with LED light RH GGB2		1
7	SP1577054	Shroud for spring bracket with luggage-rack, with LED light LH GGB2		1
8	SP1577055	Shroud for spring bracket with luggage-rack, with LED light RH GGB2		1
9	SP1577036	Shroud for spring bracket with light LH GGB2		1
10	SP1577037	Shroud for spring bracket with light RH GGB2		1

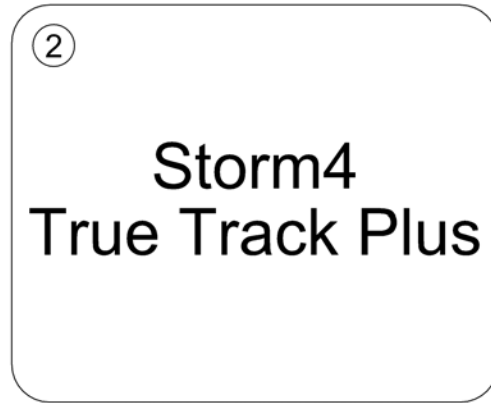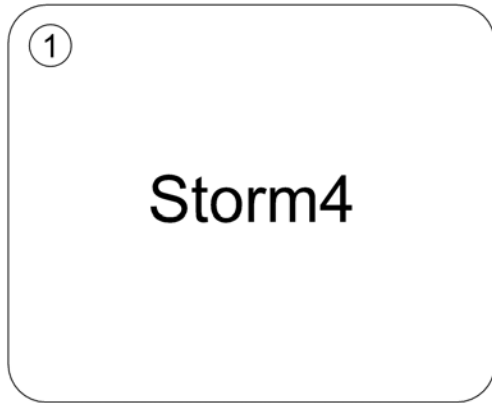

Pos.	Part number	Description	Valid from → until	Qty
1	-	Storm4		1
2	-	Storm4 TTPlus		1


Storm4

Pos.	Part number	Description	Valid from → until	Qty
1	-	Standard Seat		1
2	-	EasyAdapt Seat		1
3	-	Modulite Seat		1
4	-	Ultra Low Maxx Seat		1
5	-	Recaro Seat		1
6	-	Optimist Seat		1

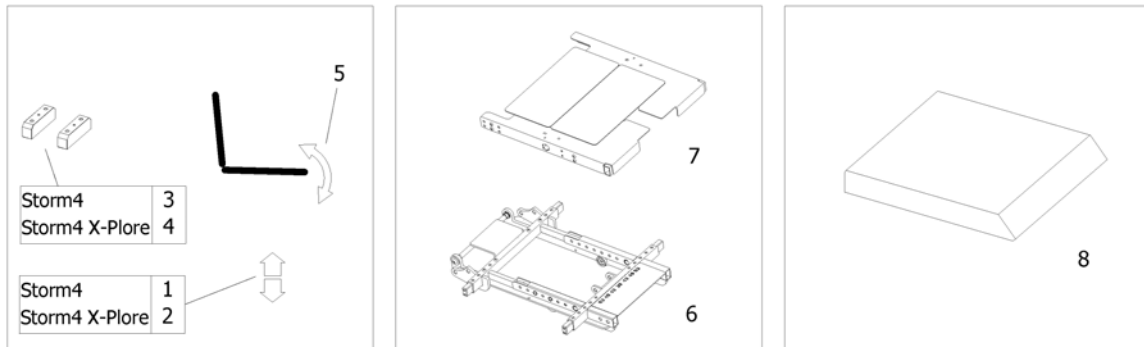


Pos.	Part number	Description	Valid from → until	Qty
1	-	Seat column / -raiser Storm4 [1025/1026/1030/1040]		1
2	-	Seat column / -raiser Storm4 X-Plore [1025/1026/1030/1040]		1
3	-	Seat Height Adapter Storm4 [1040]		1
4	-	Seat Height Adapter Storm4 X-Plore [1040]		1
5	-	Seat Tilt [1002][1014]		1
6	-	Seat Frame		1
7	-	Seat Plate		1
8	-	Seat Cushion		1

Seat column / -raiser Storm4

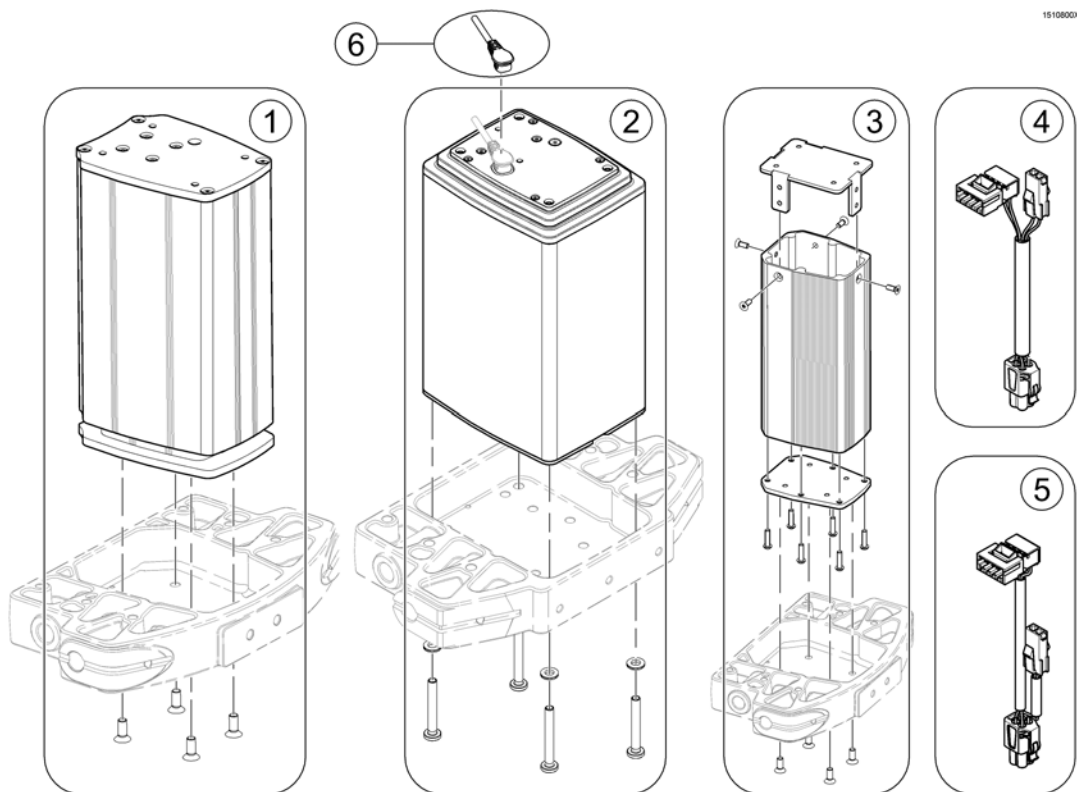

Pos.	Part number	Description	Valid from → until	Qty
1	SP1530459	Lifting column, cpl.	→ 2012-05-14	1
-	SP1568362	Mounting kit for seatbracket	→ 2012-05-14	1
2	SP1556263	Lifting column, cpl.	2012-05-15 →	1
-	SP1568361	Mounting Kit lifter for seat bracket	2012-05-15 →	1
3	SP1548536	Seat column, fixed, height 45cm and 48cm, adjustable		1
4	SP1529390	Slowdown- / Tiltcable	→ 2012-05-14	1
5	SP1555994	Slowdown- / Tiltcable	2012-05-15 →	1
6	SP1559850	Cable LINAK für Lifting Column	2012-05-15 →	1

Seat column / -raiser Storm4 X-Plore

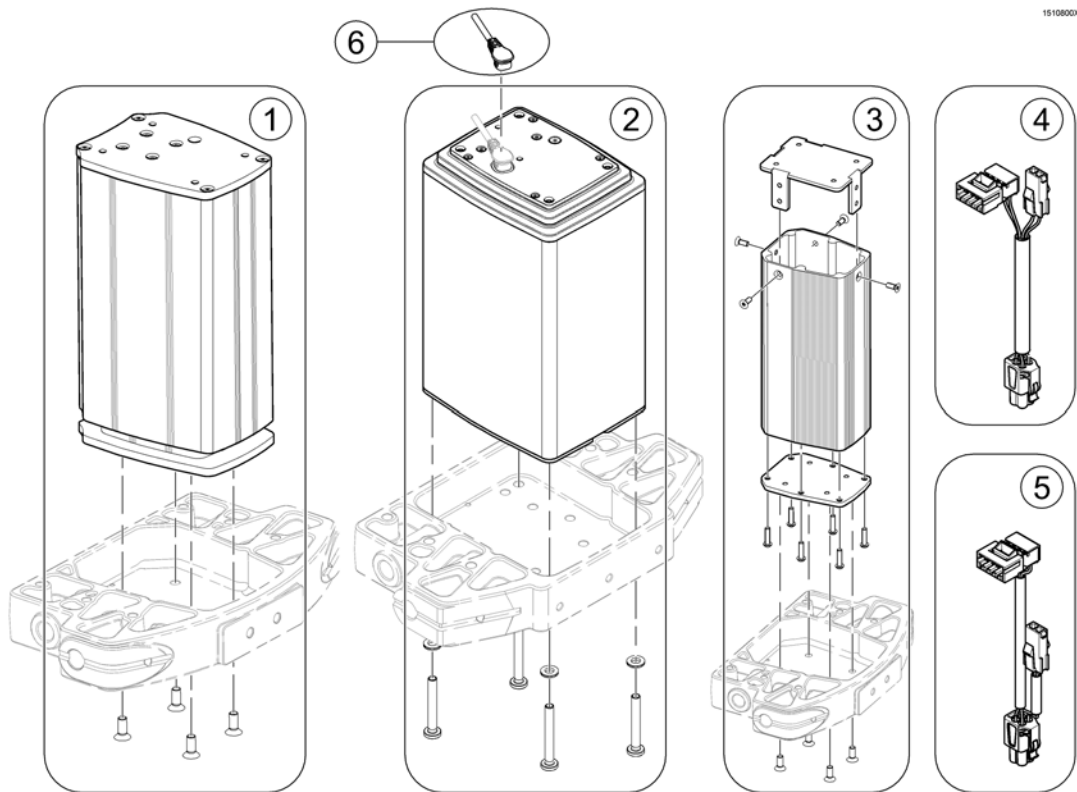


1510800X.lin

Pos.	Part number	Description	Valid from → until	Qty
1	SP1530459	Lifting column, cpl.	→ 2013-09-15	1
-	SP1568362	Mounting kit for seatbracket	→ 2013-09-15	1
2	SP1556263	Lifting column, cpl.	2013-09-16 →	1
-	SP1568361	Mounting Kit lifter for seat bracket	2013-09-16 →	1
3	SP1548536	Seat column, fixed, height 45cm and 48cm, adjustable		1
4	SP1529390	Slowdown- / Tiltcable	→ 2013-09-15	1
5	SP1555994	Slowdown- / Tiltcable	2013-09-16 →	1
6	SP1559850	Cable LINAK für Lifting Column	2013-09-16 →	1

Seat Height Adapter Storm4


1560782.iso

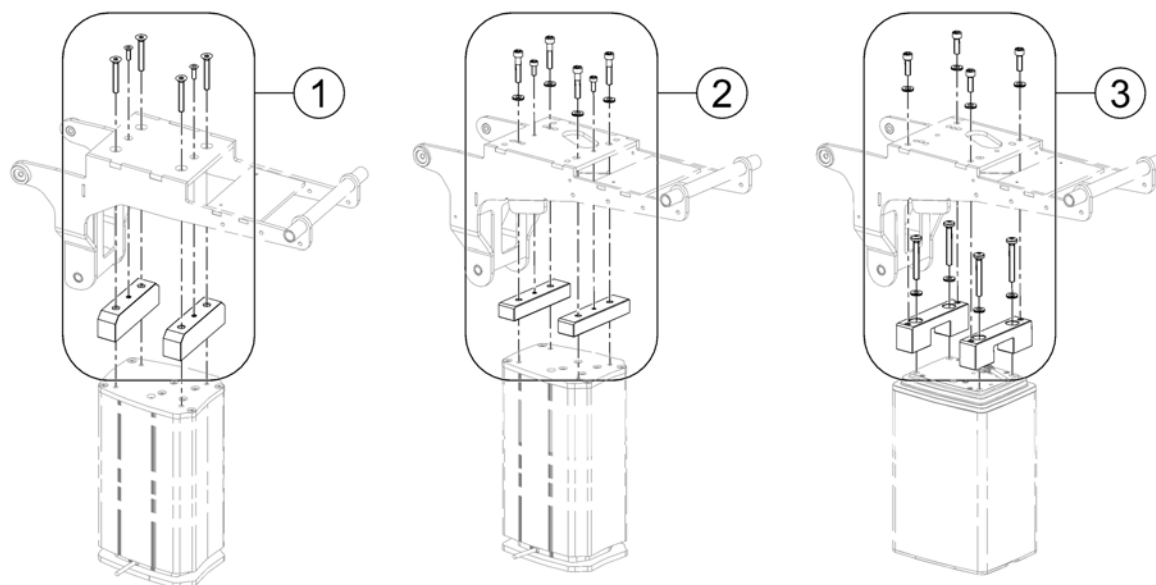

Pos.	Part number	Description	Valid from → until	Qty
1	SP1540334	Seatheightadapter cpl.	→ 2012-04-19	1
2	SP1556447	Seatheightadapter cpl.	2012-04-20 → 2012-05-14	1
3	SP1555071	Seatheightadapter cpl.	2012-05-15 →	1

Seat Height Adapter Storm4 X-Plore


1560782.iso

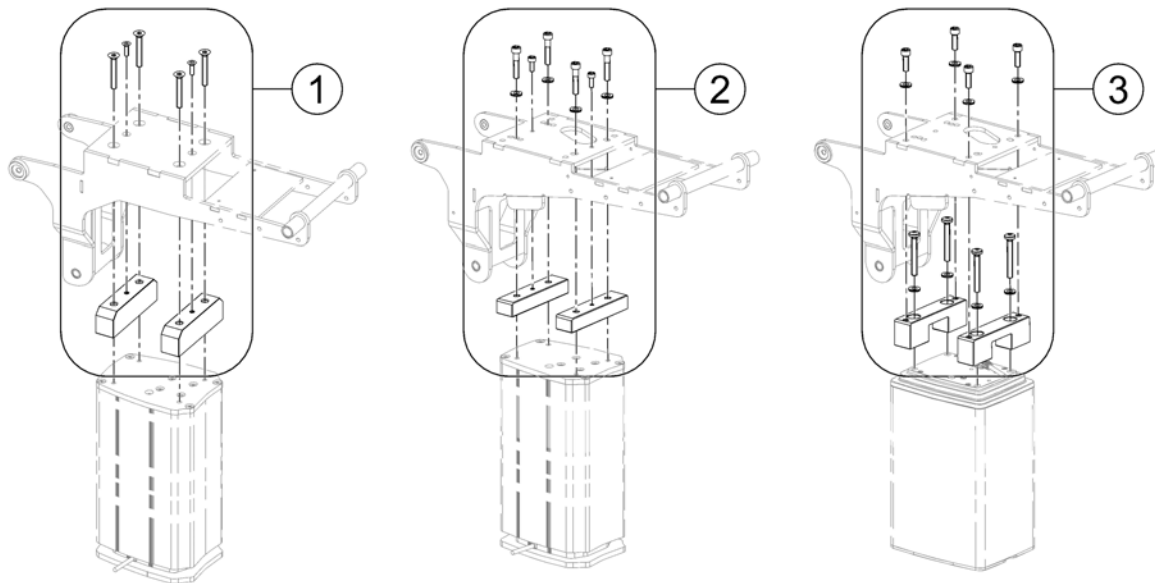

Pos.	Part number	Description	Valid from → until	Qty
1	SP1540334	Seatheightadapter cpl.	→ 2012-04-19	1
2	SP1556447	Seatheightadapter cpl.	2012-04-20 → 2013-09-15	1
3	SP1555071	Seatheightadapter cpl.	2013-09-16 →	1

Seat Tilt


1583146.ro

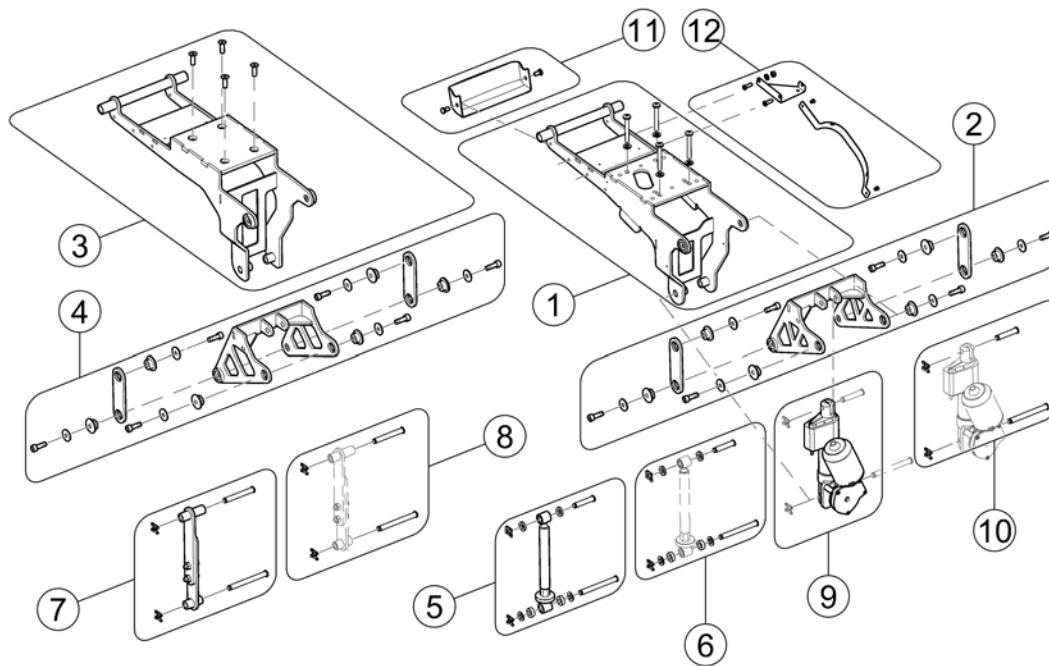

Pos.	Part number	Description	Valid from → until	Qty
1	SP1551991	Tilt carrier [1002][1014]	2012-04-20 →	1
2	SP1559711	Tiltmechanism, cpl. [1002][1014]	2012-04-20 →	1
3	-	No longer available - please refer to Replacement Info for which part is to be used instead. [1002][1014] - SP1568271	→ 2012-04-19	0
4	SP1537803	Tiltmechanism, cpl. [1002][1014]	→ 2012-04-19	1
5	SP1551010	Tilt spindle, cpl. [1002]	2011-03-18 →	1
6	SP1551009	Mounting-Kit	2011-03-18 →	1
7	SP1532482	Manual tilt, cpl. [1002]	→ 2011-03-17	1
8	SP1532483	Mounting-Kit for manual tilt	→ 2011-03-17	1
9	SP1532480	Motor for powered tilt, cpl. [1014] ACS		1
9	SP1607900	REAC Motor RE35 (w/o switch) [1014] LiNX	2017-01-01 →	1
10	SP1532481	Mounting-Kit for powered tilt		1
11	SP1588880	Cable Cover	2016-01-01 →	1
12	SP1603273	Cable Support Sheets, cpl. [1030][1040]		1


Seat Frame

1524555X.dwg

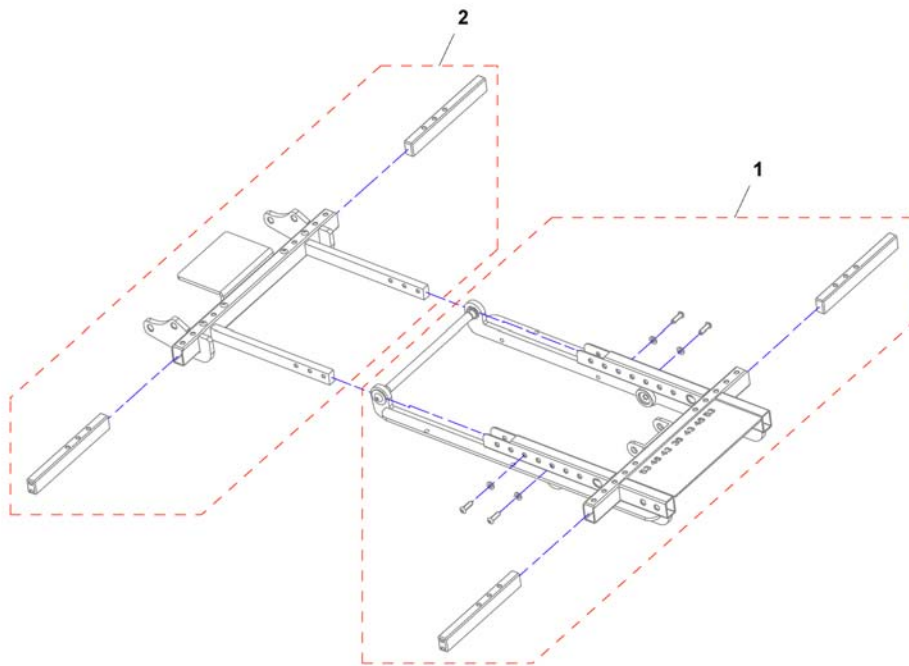


Pos.	Part number	Description	Valid from → until	Qty
1	SP1536856	Front Seat Frame		1
2	SP1542967	Rear Seat Frame for EasyAdapt /Standard - Seat		1

Seat Plate

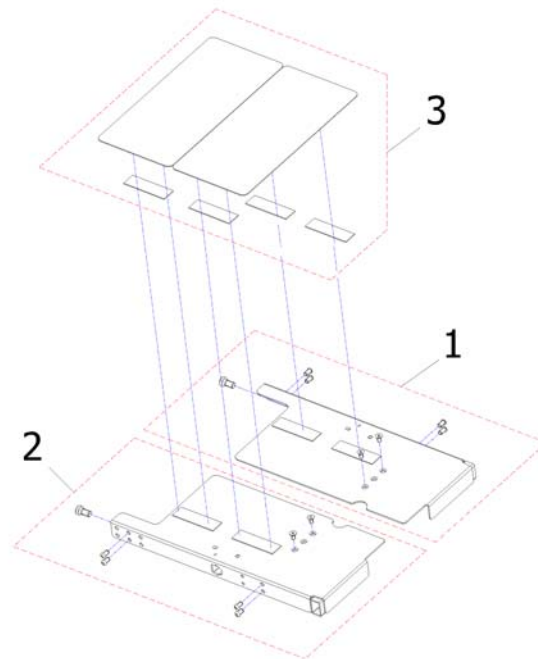

1573205.iso


Pos.	Part number	Description	Valid from → until	Qty
1	SP1530486	Seat Plate, short (440mm), LH		1
2	SP1530487	Seat Plate, short (440mm), RH		1
1	SP1530488	Seat Plate, middle (490mm), LH		1
2	SP1530489	Seat Plate, middle (490mm), RH		1
1	SP1530490	Seat Plate, lang (540mm), LH		1
2	SP1530491	Seat Plate, long (540mm), RH		1
3	SP1530492	Cover Plates		1

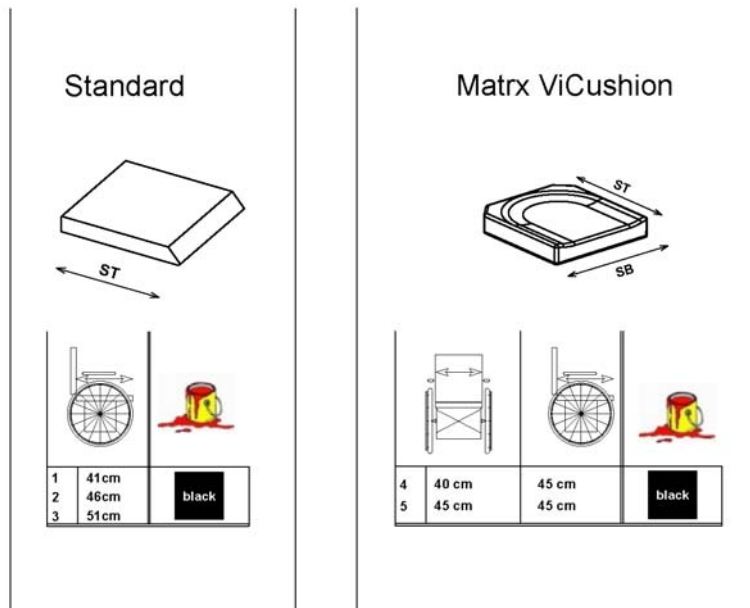

Pos.	Part number	Description	Valid from → until	Qty
1	1420460	Seat Cushion Standard - SW 430mm, SD 410mm [1336]		1
2	1420461	Seat Cushion Standard - SW 430mm, SD 460mm [1336]		1
3	1420462	Seat Cushion Standard - SW 430mm, SD 510mm [1336]		1
4	SP1531760	Matrix Vi 16x18" [2009]		1
5	SP1531761	Matrix Vi 18x18" [2011]		1

Invacare® Storm4 | Seat
EasyAdapt Seat


1573129.iss

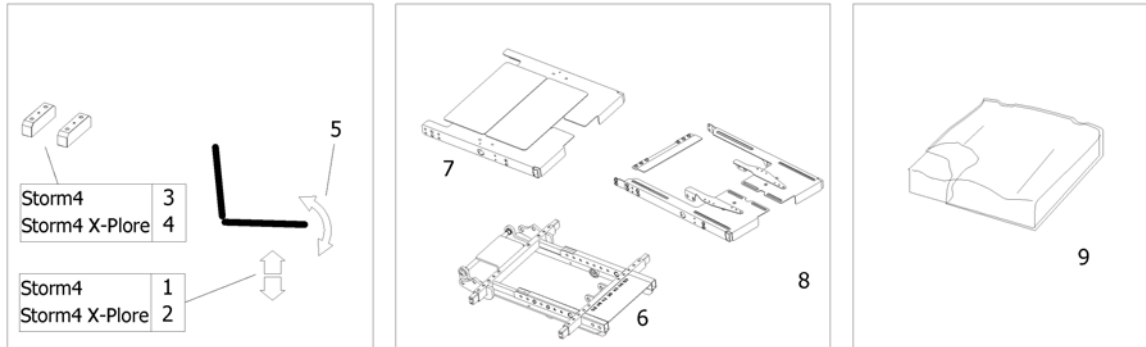

Pos.	Part number	Description	Valid from → until	Qty
1	-	Seat column / -raiser Storm4 <i>[1025/1026/1030/1040]</i>		1
2	-	Seat column / -raiser Storm4 X-Plore <i>[1025/1026/1030/1040]</i>		1
3	-	Seat Height Adapter Storm4 <i>[1040]</i>		1
4	-	Seat Height Adapter Storm4 X-Plore <i>[1040]</i>		1
5	-	Seat Tilt <i>[1002][1014]</i>		1
6	-	Seat Frame		1
7	-	Seat Plate		1
8	-	Adapter Kit for Universal Seat <i>[1064]</i>		1
9	-	Seat Cushion		1

Adapter Kit for Universal Seat

1561173.00

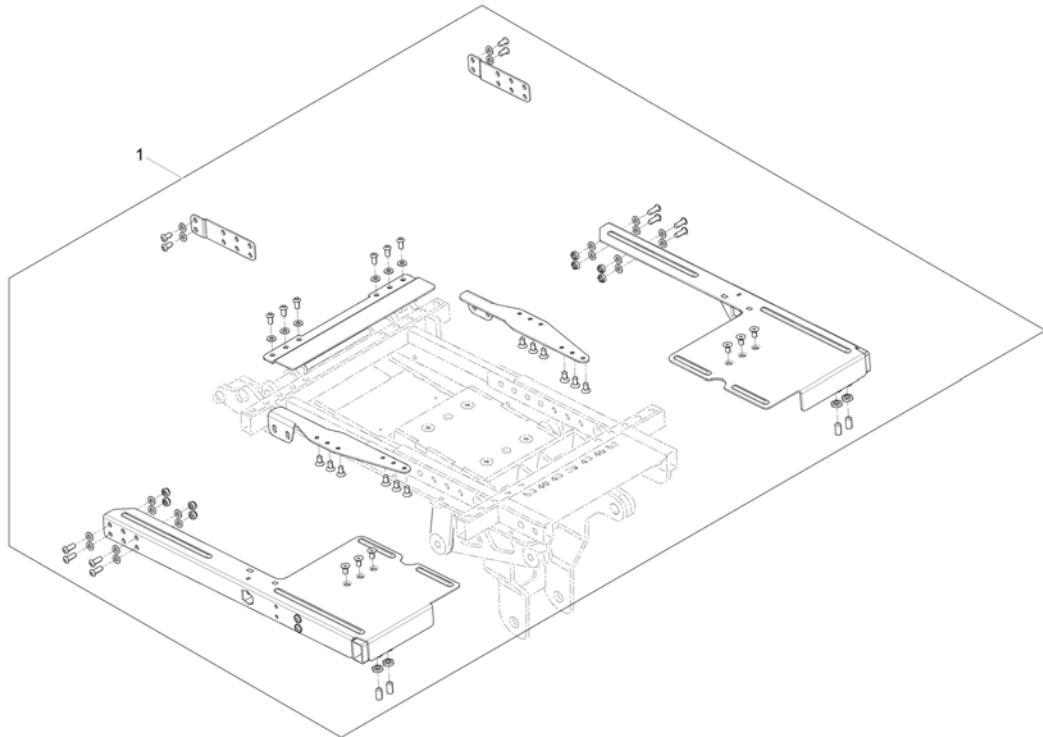


Pos.	Part number	Description	Valid from → until	Qty
1	SP1563052	Adapter Kit for Universal Seat		1


Seat Cushion

1573212/00

Matrix Flo-tech

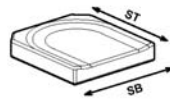

Onyx Cushion

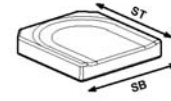

1	38cm	41cm	black smooth	25	38cm	41cm	grey	49	40cm	40cm	black																																																																																																																																																																																														
2	38cm	46cm		3	38cm	51cm		4	43cm	41cm		5	43cm	46cm	6	43cm	51cm	7	48cm	41cm	8	48cm	46cm	9	48cm	51cm	10	53cm	43cm	11	53cm	46cm	12	53cm	51cm	13	38cm	41cm	dartex grey	37	38cm	41cm	black red seam	50	40cm	45cm	black	14	38cm	46cm	15	38cm	51cm	16	43cm	41cm	17	43cm	46cm	18	43cm	51cm	19	48cm	41cm	20	48cm	46cm	21	48cm	51cm	22	53cm	43cm	23	53cm	46cm	24	53cm	51cm					38	38cm	46cm		51	45cm	40cm					39	38cm	51cm		52	45cm	45cm					40	43cm	41cm		53	51cm	40cm					41	43cm	46cm		54	51cm	45cm					42	43cm	51cm									43	48cm	41cm									44	48cm	46cm									45	48cm	51cm									46	53cm	43cm									47	53cm	46cm									48	53cm	51cm				
3	38cm	51cm		4	43cm	41cm		5	43cm	46cm		6	43cm	51cm	7	48cm	41cm	8	48cm	46cm	9	48cm	51cm	10	53cm	43cm	11	53cm	46cm	12	53cm	51cm	13	38cm	41cm	dartex grey	37	38cm		41cm	black red seam	50		40cm	45cm	black		14	38cm	46cm	15	38cm	51cm	16	43cm	41cm	17	43cm	46cm	18	43cm	51cm	19	48cm	41cm	20	48cm	46cm	21	48cm	51cm	22	53cm	43cm	23	53cm	46cm	24	53cm	51cm					38	38cm	46cm		51	45cm	40cm					39	38cm	51cm		52	45cm	45cm					40	43cm	41cm		53	51cm	40cm					41	43cm	46cm		54	51cm	45cm					42	43cm	51cm									43	48cm	41cm									44	48cm	46cm									45	48cm	51cm									46	53cm	43cm									47	53cm	46cm									48	53cm	51cm				
4	43cm	41cm		5	43cm	46cm		6	43cm	51cm		7	48cm	41cm	8	48cm	46cm	9	48cm	51cm	10	53cm	43cm	11	53cm	46cm	12	53cm	51cm	13	38cm	41cm	dartex grey	37	38cm		41cm	black red seam		50		40cm		45cm	black			14	38cm	46cm	15	38cm	51cm	16	43cm	41cm	17	43cm	46cm	18	43cm	51cm	19	48cm	41cm	20	48cm	46cm	21	48cm	51cm	22	53cm	43cm	23	53cm	46cm	24	53cm	51cm					38	38cm	46cm		51	45cm	40cm					39	38cm	51cm		52	45cm	45cm					40	43cm	41cm		53	51cm	40cm					41	43cm	46cm		54	51cm	45cm					42	43cm	51cm									43	48cm	41cm									44	48cm	46cm									45	48cm	51cm									46	53cm	43cm									47	53cm	46cm									48	53cm	51cm				
5	43cm	46cm		6	43cm	51cm		7	48cm	41cm		8	48cm	46cm	9	48cm	51cm	10	53cm	43cm	11	53cm	46cm	12	53cm	51cm	13	38cm	41cm	dartex grey	37	38cm		41cm	black red seam		50			40cm		45cm		black				14	38cm	46cm	15	38cm	51cm	16	43cm	41cm	17	43cm	46cm	18	43cm	51cm	19	48cm	41cm	20	48cm	46cm	21	48cm	51cm	22	53cm	43cm	23	53cm	46cm	24	53cm	51cm					38	38cm	46cm		51	45cm	40cm					39	38cm	51cm		52	45cm	45cm					40	43cm	41cm		53	51cm	40cm					41	43cm	46cm		54	51cm	45cm					42	43cm	51cm									43	48cm	41cm									44	48cm	46cm									45	48cm	51cm									46	53cm	43cm									47	53cm	46cm									48	53cm	51cm				
6	43cm	51cm		7	48cm	41cm		8	48cm	46cm		9	48cm	51cm	10	53cm	43cm	11	53cm	46cm	12	53cm	51cm	13	38cm	41cm	dartex grey	37	38cm		41cm	black red seam		50			40cm			45cm		black						14	38cm	46cm	15	38cm	51cm	16	43cm	41cm	17	43cm	46cm	18	43cm	51cm	19	48cm	41cm	20	48cm	46cm	21	48cm	51cm	22	53cm	43cm	23	53cm	46cm	24	53cm	51cm					38	38cm	46cm		51	45cm	40cm					39	38cm	51cm		52	45cm	45cm					40	43cm	41cm		53	51cm	40cm					41	43cm	46cm		54	51cm	45cm					42	43cm	51cm									43	48cm	41cm									44	48cm	46cm									45	48cm	51cm									46	53cm	43cm									47	53cm	46cm									48	53cm	51cm				
7	48cm	41cm		8	48cm	46cm		9	48cm	51cm	10	53cm	43cm	11	53cm	46cm	12	53cm	51cm	13	38cm	41cm	dartex grey	37	38cm	41cm		black red seam	50		40cm			45cm			black			14								38cm	46cm	15	38cm	51cm	16	43cm	41cm	17	43cm	46cm	18	43cm	51cm	19	48cm	41cm	20	48cm	46cm	21	48cm	51cm	22	53cm	43cm	23	53cm	46cm	24	53cm	51cm					38	38cm	46cm		51	45cm	40cm					39	38cm	51cm		52	45cm	45cm					40	43cm	41cm		53	51cm	40cm					41	43cm	46cm		54	51cm	45cm					42	43cm	51cm									43	48cm	41cm									44	48cm	46cm									45	48cm	51cm									46	53cm	43cm									47	53cm	46cm									48	53cm	51cm					
8	48cm	46cm		9	48cm	51cm		10	53cm	43cm	11	53cm	46cm	12	53cm	51cm	13	38cm	41cm	dartex grey	37	38cm		41cm	black red seam	50			40cm		45cm			black						14							38cm	46cm	15	38cm	51cm	16	43cm	41cm	17	43cm	46cm	18	43cm	51cm	19	48cm	41cm	20	48cm	46cm	21	48cm	51cm	22	53cm	43cm	23	53cm	46cm	24	53cm	51cm					38	38cm	46cm		51	45cm	40cm					39	38cm	51cm		52	45cm	45cm					40	43cm	41cm		53	51cm	40cm					41	43cm	46cm		54	51cm	45cm					42	43cm	51cm									43	48cm	41cm									44	48cm	46cm									45	48cm	51cm									46	53cm	43cm									47	53cm	46cm									48	53cm	51cm						
9	48cm	51cm		10	53cm	43cm		11	53cm	46cm	12	53cm	51cm	13	38cm	41cm	dartex grey	37	38cm		41cm	black red seam		50		40cm			45cm		black									14						38cm	46cm	15	38cm	51cm	16	43cm	41cm	17	43cm	46cm	18	43cm	51cm	19	48cm	41cm	20	48cm	46cm	21	48cm	51cm	22	53cm	43cm	23	53cm	46cm	24	53cm	51cm					38	38cm	46cm		51	45cm	40cm					39	38cm	51cm		52	45cm	45cm					40	43cm	41cm		53	51cm	40cm					41	43cm	46cm		54	51cm	45cm					42	43cm	51cm									43	48cm	41cm									44	48cm	46cm									45	48cm	51cm									46	53cm	43cm									47	53cm	46cm									48	53cm	51cm							
10	53cm	43cm		11	53cm	46cm		12	53cm	51cm	13	38cm	41cm	dartex grey	37	38cm		41cm	black red seam		50			40cm		45cm			black											14					38cm	46cm	15	38cm	51cm	16	43cm	41cm	17	43cm	46cm	18	43cm	51cm	19	48cm	41cm	20	48cm	46cm	21	48cm	51cm	22	53cm	43cm	23	53cm	46cm	24	53cm	51cm					38	38cm	46cm		51	45cm	40cm					39	38cm	51cm		52	45cm	45cm					40	43cm	41cm		53	51cm	40cm					41	43cm	46cm		54	51cm	45cm					42	43cm	51cm									43	48cm	41cm									44	48cm	46cm									45	48cm	51cm									46	53cm	43cm									47	53cm	46cm									48	53cm	51cm								
11	53cm	46cm		12	53cm	51cm		13	38cm	41cm	dartex grey	37	38cm		41cm	black red seam		50			40cm			45cm		black														14				38cm	46cm	15	38cm	51cm	16	43cm	41cm	17	43cm	46cm	18	43cm	51cm	19	48cm	41cm	20	48cm	46cm	21	48cm	51cm	22	53cm	43cm	23	53cm	46cm	24	53cm	51cm					38	38cm	46cm		51	45cm	40cm					39	38cm	51cm		52	45cm	45cm					40	43cm	41cm		53	51cm	40cm					41	43cm	46cm		54	51cm	45cm					42	43cm	51cm									43	48cm	41cm									44	48cm	46cm									45	48cm	51cm									46	53cm	43cm									47	53cm	46cm									48	53cm	51cm									
12	53cm	51cm		13	38cm	41cm		dartex grey	37	38cm		41cm	black red seam		50			40cm			45cm			black																14		38cm		46cm	15	38cm	51cm	16	43cm	41cm	17	43cm	46cm	18	43cm	51cm	19	48cm	41cm	20	48cm	46cm	21	48cm	51cm	22	53cm	43cm	23	53cm	46cm	24	53cm	51cm					38	38cm	46cm		51	45cm	40cm					39	38cm	51cm		52	45cm	45cm					40	43cm	41cm		53	51cm	40cm					41	43cm	46cm		54	51cm	45cm					42	43cm	51cm									43	48cm	41cm									44	48cm	46cm									45	48cm	51cm									46	53cm	43cm									47	53cm	46cm									48	53cm	51cm										
13	38cm	41cm	dartex grey	37	38cm	41cm	black red seam		50	40cm		45cm			black																																																																																																																																																																																										
14	38cm	46cm		15	38cm	51cm			16	43cm		41cm						17			43cm													46cm			18		43cm	51cm		19	48cm	41cm	20	48cm	46cm	21	48cm	51cm	22	53cm	43cm	23	53cm	46cm	24	53cm	51cm					38	38cm	46cm		51	45cm	40cm					39	38cm	51cm		52	45cm	45cm					40	43cm	41cm		53	51cm	40cm					41	43cm	46cm		54	51cm	45cm					42	43cm	51cm									43	48cm	41cm									44	48cm	46cm									45	48cm	51cm									46	53cm	43cm									47	53cm	46cm									48	53cm	51cm																									
15	38cm	51cm		16	43cm	41cm			17	43cm		46cm						18			43cm										51cm			19		48cm	41cm		20	48cm	46cm	21	48cm	51cm	22	53cm	43cm	23	53cm	46cm	24	53cm	51cm					38	38cm	46cm		51	45cm	40cm					39	38cm	51cm		52	45cm	45cm					40	43cm	41cm		53	51cm	40cm					41	43cm	46cm		54	51cm	45cm					42	43cm	51cm									43	48cm	41cm									44	48cm	46cm									45	48cm	51cm									46	53cm	43cm									47	53cm	46cm									48	53cm	51cm																															
16	43cm	41cm		17	43cm	46cm			18	43cm		51cm						19			48cm								41cm		20		48cm	46cm		21	48cm	51cm	22	53cm	43cm	23	53cm	46cm	24	53cm	51cm					38	38cm	46cm		51	45cm	40cm					39	38cm	51cm		52	45cm	45cm					40	43cm	41cm		53	51cm	40cm					41	43cm	46cm		54	51cm	45cm					42	43cm	51cm									43	48cm	41cm									44	48cm	46cm									45	48cm	51cm									46	53cm	43cm									47	53cm	46cm									48	53cm	51cm																																					
17	43cm	46cm		18	43cm	51cm			19	48cm		41cm						20			48cm					46cm			21	48cm	51cm		22	53cm	43cm	23	53cm	46cm	24	53cm	51cm					38	38cm	46cm		51	45cm	40cm					39	38cm	51cm		52	45cm	45cm					40	43cm	41cm		53	51cm	40cm					41	43cm	46cm		54	51cm	45cm					42	43cm	51cm									43	48cm	41cm									44	48cm	46cm									45	48cm	51cm									46	53cm	43cm									47	53cm	46cm									48	53cm	51cm																																											
18	43cm	51cm		19	48cm	41cm			20	48cm		46cm						21			48cm			51cm		22	53cm		43cm	23	53cm	46cm	24	53cm	51cm					38	38cm	46cm		51	45cm	40cm					39	38cm	51cm		52	45cm	45cm					40	43cm	41cm		53	51cm	40cm					41	43cm	46cm		54	51cm	45cm					42	43cm	51cm									43	48cm	41cm									44	48cm	46cm									45	48cm	51cm									46	53cm	43cm									47	53cm	46cm									48	53cm	51cm																																																	
19	48cm	41cm		20	48cm	46cm			21	48cm		51cm			22			53cm			43cm		23	53cm		46cm	24	53cm	51cm					38	38cm	46cm		51	45cm	40cm					39	38cm	51cm		52	45cm	45cm					40	43cm	41cm		53	51cm	40cm					41	43cm	46cm		54	51cm	45cm					42	43cm	51cm									43	48cm	41cm									44	48cm	46cm									45	48cm	51cm									46	53cm	43cm									47	53cm	46cm									48	53cm	51cm																																																							
20	48cm	46cm		21	48cm	51cm			22	53cm		43cm			23			53cm		46cm	24		53cm	51cm					38	38cm	46cm		51	45cm	40cm					39	38cm	51cm		52	45cm	45cm					40	43cm	41cm		53	51cm	40cm					41	43cm	46cm		54	51cm	45cm					42	43cm	51cm									43	48cm	41cm									44	48cm	46cm									45	48cm	51cm									46	53cm	43cm									47	53cm	46cm									48	53cm	51cm																																																												
21	48cm	51cm		22	53cm	43cm			23	53cm		46cm			24		53cm	51cm						38	38cm	46cm		51	45cm	40cm					39	38cm	51cm		52	45cm	45cm					40	43cm	41cm		53	51cm	40cm					41	43cm	46cm		54	51cm	45cm					42	43cm	51cm									43	48cm	41cm									44	48cm	46cm									45	48cm	51cm									46	53cm	43cm									47	53cm	46cm									48	53cm	51cm																																																																	
22	53cm	43cm		23	53cm	46cm			24	53cm		51cm							38	38cm	46cm		51	45cm	40cm					39	38cm	51cm		52	45cm	45cm					40	43cm	41cm		53	51cm	40cm					41	43cm	46cm		54	51cm	45cm					42	43cm	51cm									43	48cm	41cm									44	48cm	46cm									45	48cm	51cm									46	53cm	43cm									47	53cm	46cm									48	53cm	51cm																																																																						
23	53cm	46cm		24	53cm	51cm								38	38cm	46cm		51	45cm	40cm					39	38cm	51cm		52	45cm	45cm					40	43cm	41cm		53	51cm	40cm					41	43cm	46cm		54	51cm	45cm					42	43cm	51cm									43	48cm	41cm									44	48cm	46cm									45	48cm	51cm									46	53cm	43cm									47	53cm	46cm									48	53cm	51cm																																																																											
24	53cm	51cm							38	38cm	46cm		51	45cm	40cm					39	38cm	51cm		52	45cm	45cm					40	43cm	41cm		53	51cm	40cm					41	43cm	46cm		54	51cm	45cm					42	43cm	51cm									43	48cm	41cm									44	48cm	46cm									45	48cm	51cm									46	53cm	43cm									47	53cm	46cm									48	53cm	51cm																																																																																
				38	38cm	46cm		51	45cm	40cm																																																																																																																																																																																															
				39	38cm	51cm		52	45cm	45cm																																																																																																																																																																																															
				40	43cm	41cm		53	51cm	40cm																																																																																																																																																																																															
				41	43cm	46cm		54	51cm	45cm																																																																																																																																																																																															
				42	43cm	51cm																																																																																																																																																																																																			
				43	48cm	41cm																																																																																																																																																																																																			
				44	48cm	46cm																																																																																																																																																																																																			
				45	48cm	51cm																																																																																																																																																																																																			
				46	53cm	43cm																																																																																																																																																																																																			
				47	53cm	46cm																																																																																																																																																																																																			
				48	53cm	51cm																																																																																																																																																																																																			


Pos.	Part number	Description	Valid from → until	Qty
1	SPFLB15/16	Flo-Tech Lite 15x16"		1
2	SPFLB15/18	Seatcushion Easy-Adapt SB38 x ST46cm black smooth		1
3	SPFLB15/20	Seatcushion Easy-Adapt SB38 x ST51cm black smooth		1
4	SPFLB17/16	Seatcushion Easy-Adapt SB43 x ST41cm black smooth		1
5	SPFLB17/18	Seatcushion Easy-Adapt SB43 x ST46cm black smooth		1
6	SPFLB17/20	Seatcushion Easy-Adapt SB43 x ST51cm black smooth		1
7	SPFLB19/16	Seatcushion Easy-Adapt SB48 x ST41cm black smooth		1
8	SPFLB19/18	Seatcushion Easy-Adapt SB48 x ST46cm black smooth		1
9	SPFLB19/20	Flo-Tech Lite 19x20"		1
10	SPFCB21/17	Flo-Tech Contour 21x17"		1
11	SPFCB21/18	Flo-Tech Contour 21x18"		1
12	SPFCB21/20	Flo-Tech Contour 21x20"		1
13	SP1525250-0232	Seatcushion Easy-Adapt SB38 x ST41cm dartex grey	→ 2014-11-30	1
13	SP1529546-TR26	seat cushion Flo-tech Lite SW 38 x SD 41 Dartex black	2014-12-01 →	1
14	SP1525252-0232	Seatcushion Easy-Adapt SB38 x ST46cm dartex grey	→ 2014-11-30	1
14	SP1529547-TR26	seat cushion Flo-tech Lite SW 38 x SD 46 Dartex black	2014-12-01 →	1
15	SP1525253-0232	Seatcushion Easy-Adapt SB38 x ST51cm dartex grey	→ 2014-11-30	1
15	SP1529548-TR26	seat cushion Flo-tech Lite SW 38 x SD 51 Dartex black	2014-12-01 →	1
16	SP1525254-0232	Seatcushion Easy-Adapt SB43 x ST41cm dartex grey	→ 2014-11-30	1
16	SP1529549-TR26	seat cushion Flo-tech Lite SW 43 x SD 41 Dartex black	2014-12-01 →	1
17	-	No longer available - please choose Dartex black	→ 2014-11-30	1
17	SP1529550-TR26	seat cushion Flo-tech Lite SW 43 x SD 46 Dartex black	2014-12-01 →	1
18	SP1525256-0232	Seatcushion Easy-Adapt SB43 x ST51cm dartex grey	→ 2014-11-30	1
18	SP1529551-TR26	seat cushion Flo-tech Lite SW 43 x SD 51 Dartex black	2014-12-01 →	1
19	SP1525257-0232	Seatcushion Easy-Adapt SB48 x ST41cm dartex grey	→ 2014-11-30	1
19	SP1529552-TR26	seat cushion Flo-tech Lite SW 48 x SD 41 Dartex black	2014-12-01 →	1
20	SP1525258-0232	Seatcushion Easy-Adapt SB48 x ST46cm dartex grey	→ 2014-11-30	1
20	SP1529553-TR26	seat cushion Flo-tech Lite SW 48 x SD 46 Dartex black	2014-12-01 →	1
21	SP1525259-0232	Easy-Adapt cushion (Flo-tech Lite) SW 48 x SD 51cm Dartex	→ 2014-11-30	1
21	SP1529554-TR26	seat cushion Flo-tech Lite SW 48 x SD 51 Dartex black	2014-12-01 →	1
22	SP1525260-0232	Seatcushion Easy-Adapt SB53 x ST43cm dartex grey	→ 2014-11-30	1

Seat Cushion

Pos.	Part number	Description	Valid from → until	Qty
22	SP1529555-TR26	seat cushion Flo-tech Contour SW 53 x SD 43 Dartex black	2014-12-01 →	1
23	SP1525261-0232	Seatcushion Easy-Adapt SB53 x ST46cm dartex grey	→ 2014-11-30	1
23	SP1529556-TR26	seat cushion Flo-tech Contour SW 53 x SD 46 Dartex black	2014-12-01 →	1
24	SP1525262-0232	Seatcushion Easy-Adapt SB53 x ST51cm dartex grey	→ 2014-11-30	1
24	SP1529570-TR26	seat cushion Flo-tech Contour SW 53 x SD 51 Dartex black	2014-12-01 →	1
25	SP1525250-0237	Seatcushion Easy-Adapt SB38 x ST41cm grey plush		1
26	SP1525252-0237	Seatcushion Easy-Adapt SB38 x ST46cm grey plush		1
27	SP1525253-0237	Seatcushion Easy-Adapt SB38 x ST51cm grey plush		1
28	SP1525254-0237	Seatcushion Easy-Adapt SB43 x ST41cm grey plush		1
29	SP1525255-0237	Seatcushion Easy-Adapt SB43 x ST46cm grey plush		1
30	SP1525256-0237	Seatcushion Easy-Adapt SB43 x ST51cm grey plush		1
31	SP1525257-0237	Seatcushion Easy-Adapt SB48 x ST41cm grey plush		1
32	SP1525258-0237	Seatcushion Easy-Adapt SB48 x ST46cm grey plush		1
33	SP1525259-0237	Easy-Adapt cushion (Flo-tech Lite) SW 48 x SD 51cm Grey Plush		1
34	SP1525260-0237	Seatcushion Easy-Adapt SB53 x ST43cm grey plush		1
35	SP1525261-0237	Seatcushion Easy-Adapt SB53 x ST46cm grey plush		1
36	SP1525262-0237	Seatcushion Easy-Adapt SB53 x ST51cm grey plush		1
37	SP1529546	Seatcushion Easy-Adapt SB38 x ST41cm black with red seam		1
38	SP1529547	Seatcushion Easy-Adapt SB38 x ST46cm black with red seam		1
39	SP1529548	Seatcushion Easy-Adapt SB38 x ST51cm black with red seam		1
40	SP1529549	Seatcushion Easy-Adapt SB43 x ST41cm black with red seam		1
41	SP1529550	Seatcushion Easy-Adapt SB43 x ST46cm black with red seam		1
42	SP1529551	Seatcushion Easy-Adapt SB43 x ST51cm black with red seam		1
43	SP1529552	Seatcushion Easy-Adapt SB48 x ST41cm black with red seam		1
44	SP1529553	Seatcushion Easy-Adapt SB48 x ST46cm black with red seam		1
45	SP1529554	Seatcushion Easy-Adapt SB48 x ST51cm black with red seam		1
46	SP1529555	Seatcushion Easy-Adapt SB53 x ST43cm black with red seam		1
47	SP1529556	Seatcushion Easy-Adapt SB53 x ST46cm black with red seam		1
48	SP1529570	Seatcushion Easy-Adapt SB53 x ST51cm black with red seam		1
49	SP1532058	ONYX Seat Cushion 40x40cm (16"x16")		1
50	SP1534080	ONYX Seat Cushion 40x45cm (16"x18")		1
51	SP1532059	ONYX Seat Cushion 45x40cm (18"x16")		1
52	SP1532060	ONYX Seat Cushion 45x45cm (18"x18")		1
53	SP1534136	ONYX Seat Cushion 51x40cm (20"x16")		1
54	SP1534199	ONYX Seat Cushion 51x45cm (20"x18")		1

Modulite Seat


1573130.iss

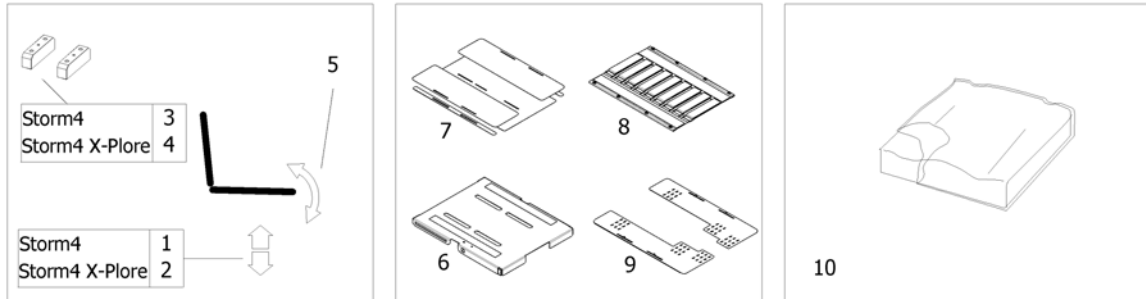

Pos.	Part number	Description	Valid from → until	Qty
1	-	Seat column / -raiser Storm4 [1023/1025/1030/1040]		1
2	-	Seat column / -raiser Storm4 X-Plore [1023/1025/1030/1040]		1
3	-	Seat Height Adapter Storm4 [1040]		1
4	-	Seat Height Adapter Storm4 X-Plore [1040]		1
5	-	Seat Tilt [1002][1014]		1
6	-	One Piece Seat Plate [2110]		1
7	-	Telescopic Seat Frame and Plate [2111]		1
8	-	Telescopic Seat Frame and Sling Seat [2112]		1
9	-	Telescopic Seat Frame and Universal Adapter [2109]		1
10	-	Seat Cushions "Standard" & "Matrx Flo-Tech" [2141/2142/2143]		1
10	-	Seat Cushions "Matrx Libra" [2147]		1
10	-	Seat Cushions "Matrx Vi" [2144]		1

Seat Tilt


1583167.indd

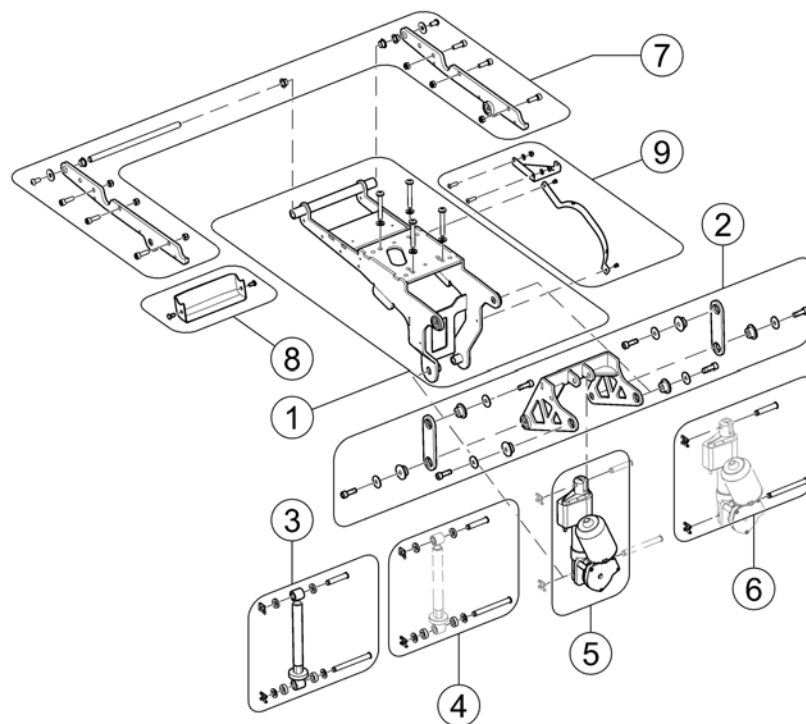

Pos.	Part number	Description	Valid from → until	Qty
1	SP1551991	Tilt carrier [1002][1014]		1
2	SP1559711	Tiltmechanism, cpl. [1002][1014]		1
3	SP1551010	Tilt spindle, cpl. [1002]		1
4	SP1551009	Mounting-Kit		1
5	SP1532480	Motor for powered tilt, cpl. [1014] ACS		1
5	SP1607900	REAC Motor RE35 (w/o switch) [1014] LiNX	2017-01-01 →	1
6	SP1532481	Mounting-Kit for powered tilt		1
7	SP1573217	Modulite Seat Support [2001]		1
8	SP1588880	Cable Cover	2016-01-01 →	1
9	SP1603273	Cable Support Sheets, cpl. [1030][1040]		1

One Piece Seat Plate


1571064.000

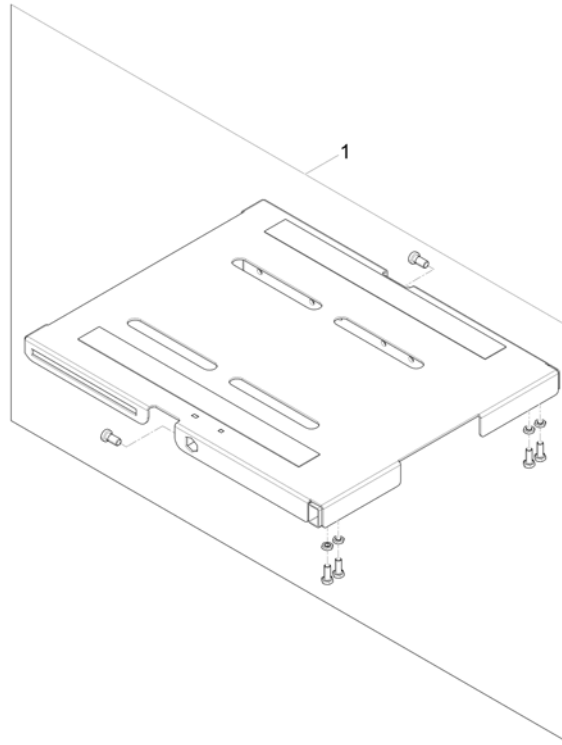

Pos.	Part number	Description	Valid from → until	Qty
1	SP1571297	One Piece Seat Plate		1

Telescopic Seat Frame and Plate


1574269 iso

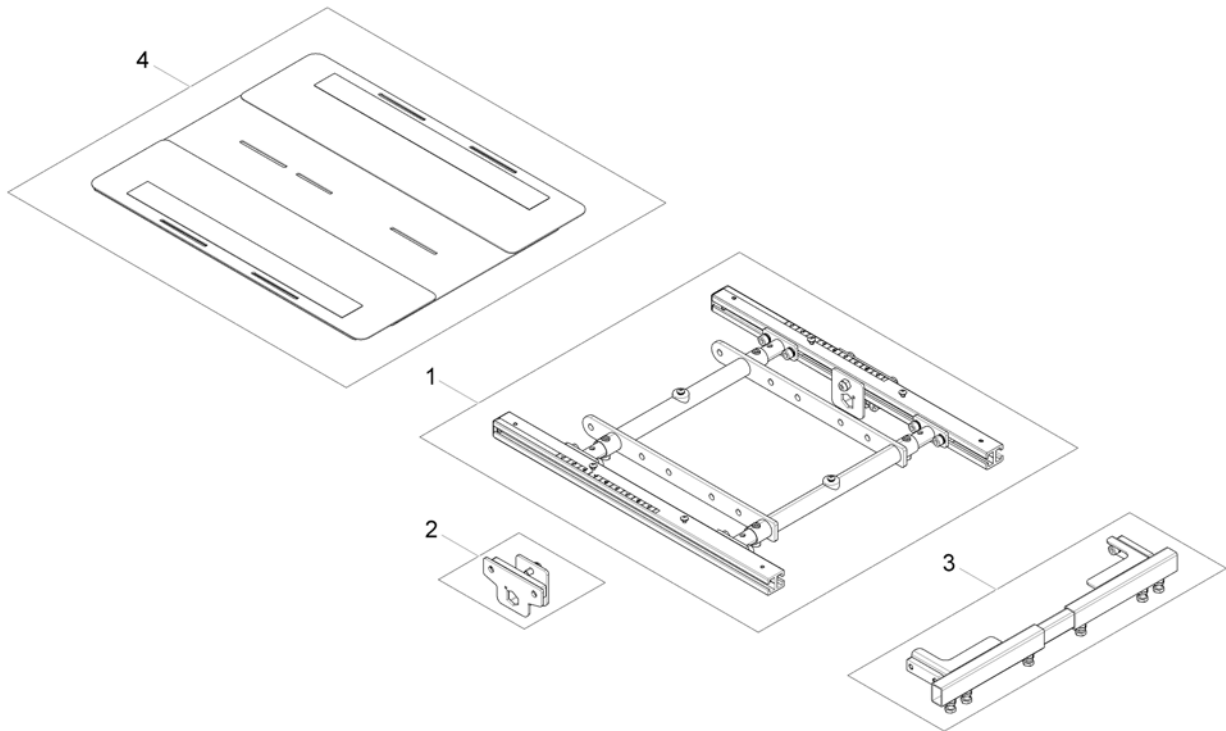

Pos.	Part number	Description	Valid from → until	Qty
1	-	Telescopic Seat Frame		1
2	-	Armrest bracket		1
3	-	Legrest Support		1
4	-	Seat Plate		1


Telescopic Seat Frame

1572098.iss

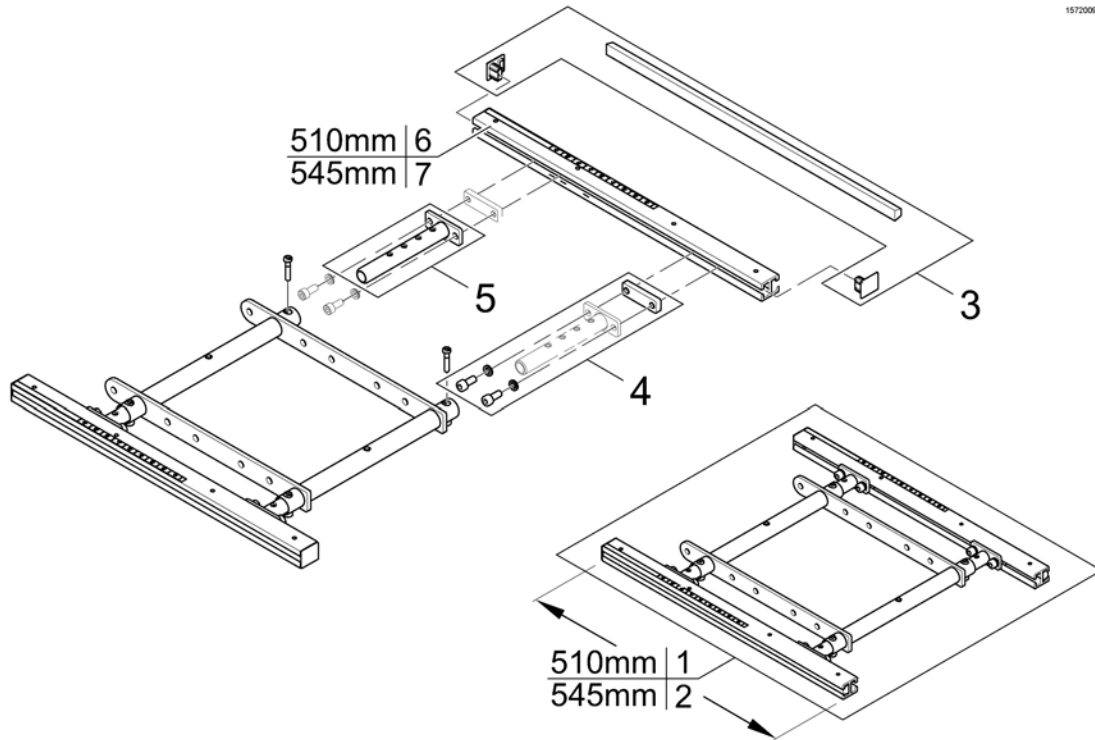

Pos.	Part number	Description	Valid from → until	Qty
1	SP1572977	Seatframe 510mm, cpl., pre set up to SW 43cm		1
2	SP1582216	Seatframe 545mm, cpl., pre set up to SW 43cm		1
3	SP1571810	Sealing strap with plugs		1
4	SP1571811	Groove stone with mounting parts		1
5	SP1554646	Adjustment Tube		1
6	SP1588443	Seat Rails 510mm (2pcs kit)		1
7	SP1588444	Seat Rails 545mm (2pcs kit)		1


1571834 Rev

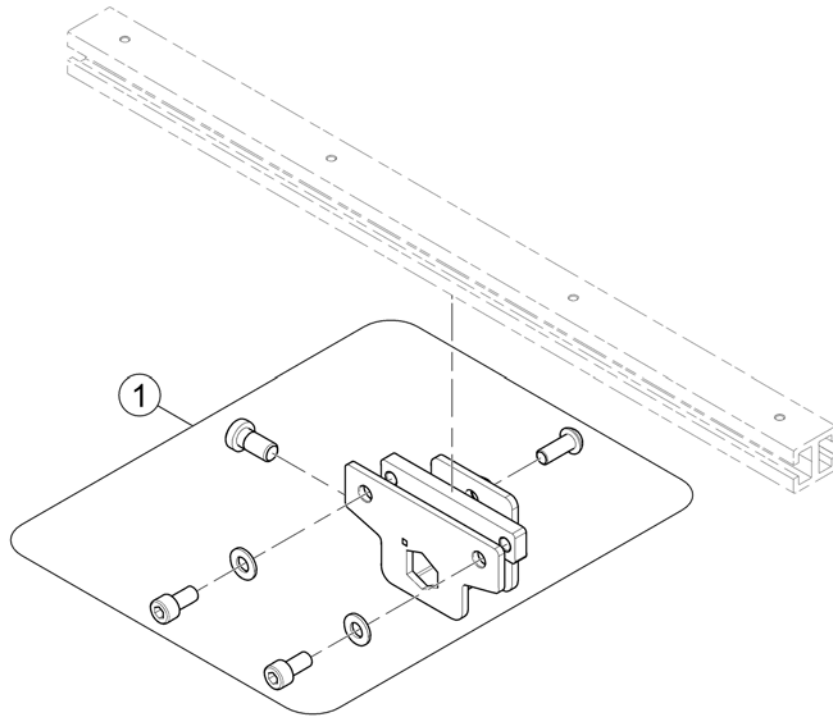


Pos.	Part number	Description	Valid from → until	Qty
1	SP1570189	Armrest bracket		1

Invacare® Storm4 | Seat
Legrest Support

1572021.000

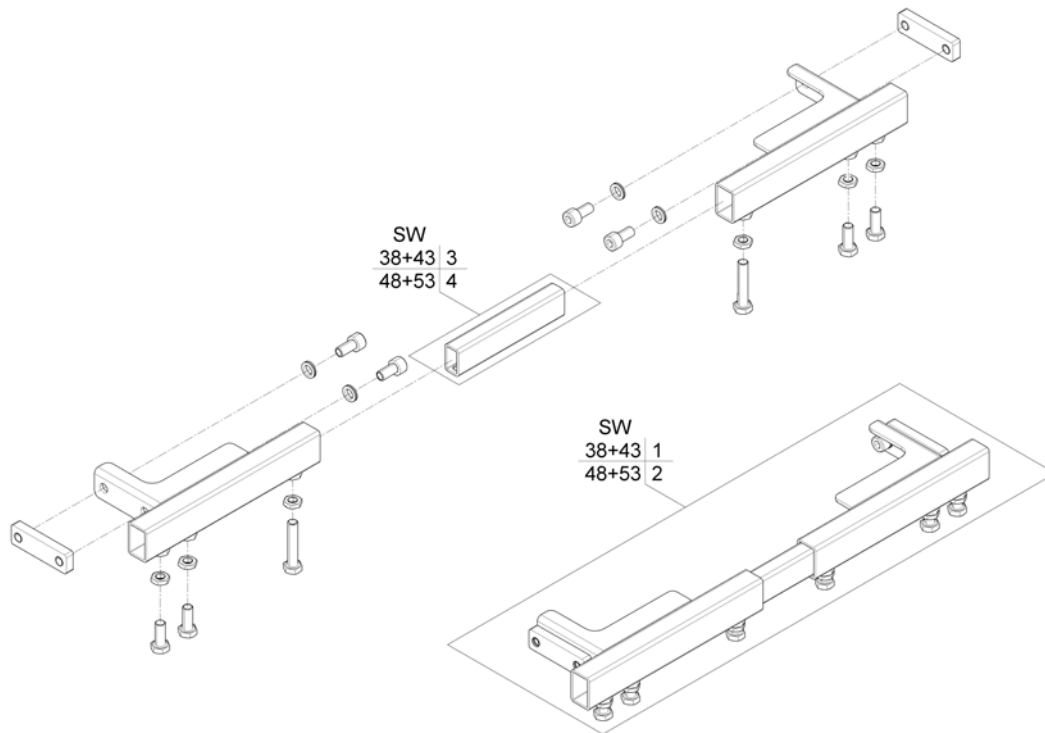


Pos.	Part number	Description	Valid from → until	Qty
1	SP1558935	Legrest Support SW38+43, cpl.		1
2	SP1559294	Legrest Support SW48+53, cpl.		1
3	SP1558134	Connection Tube SW380+430mm		1
4	SP1558135	Connection Tube SW480+530mm		1

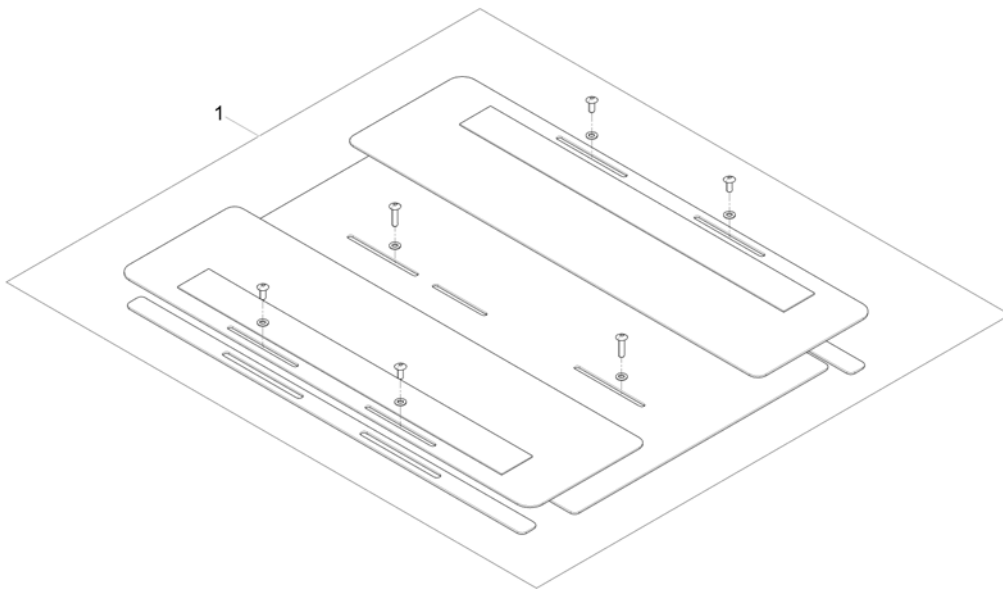

Pos.	Part number	Description	Valid from → until	Qty
1	SP1572199	Seat Plate 510mm, cpl.		1

Telescopic Seat Frame and Sling Seat


1574270.00

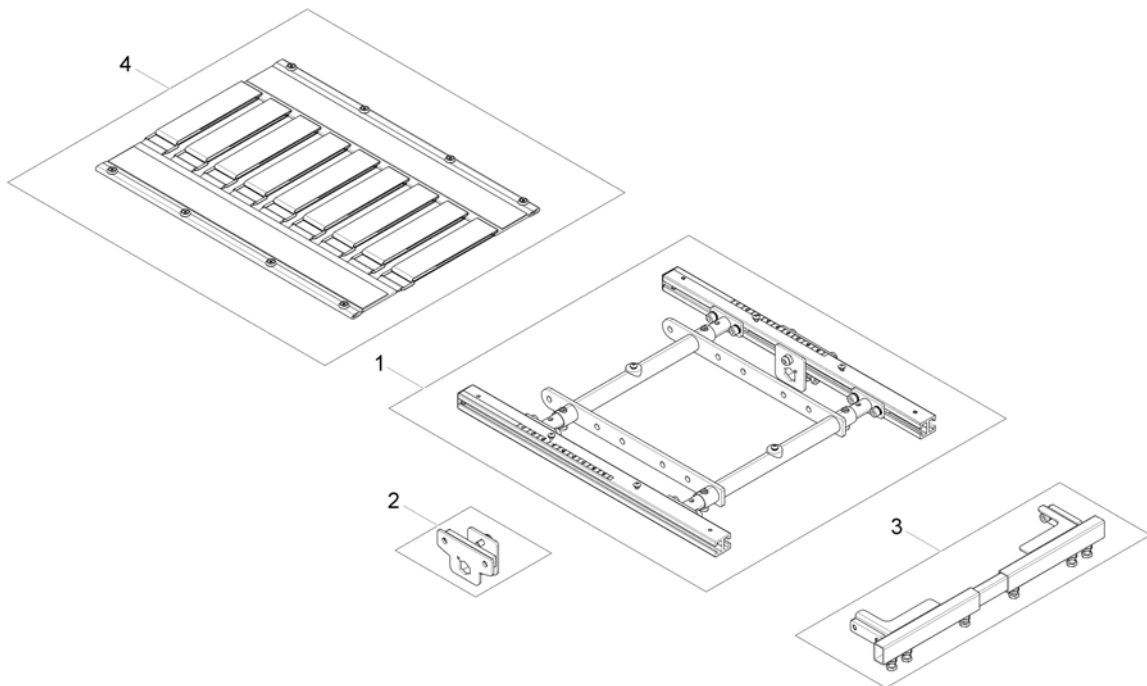


Pos.	Part number	Description	Valid from → until	Qty
1	-	Telescopic Seat Frame		1
2	-	Armrest bracket		1
3	-	Legrest Support		1
4	-	Sling Seat		1

Telescopic Seat Frame


1571950.iss

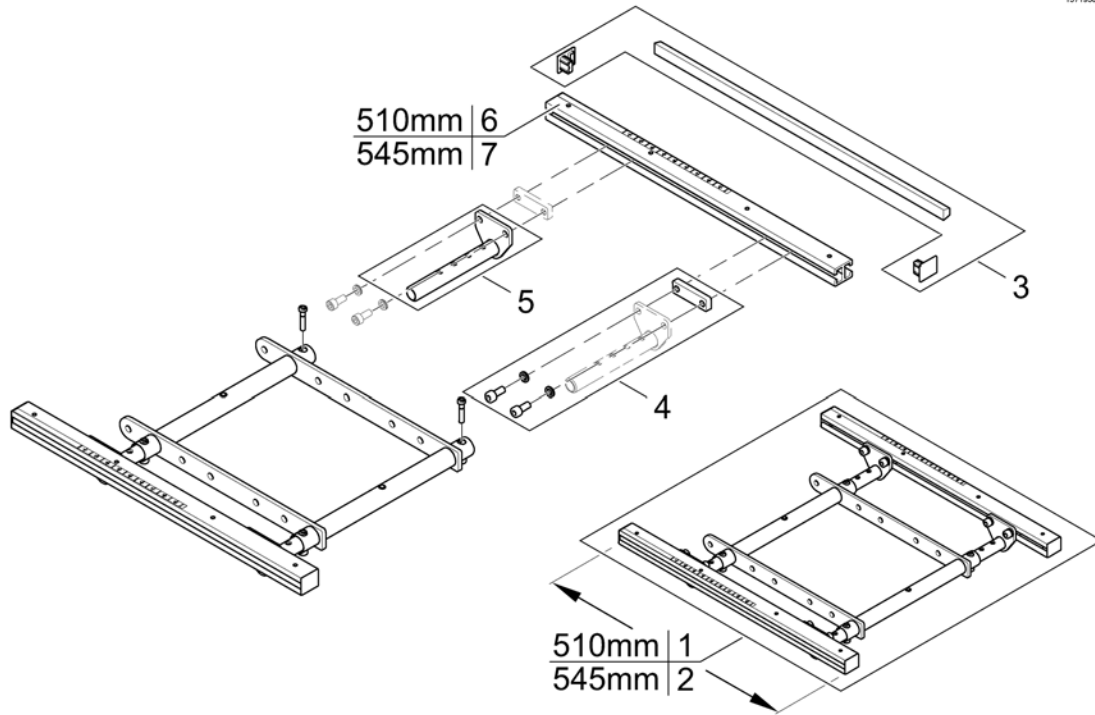

Pos.	Part number	Description	Valid from → until	Qty
1	SP1572978	Seatframe 510mm, cpl., pre set up to SW 43cm		1
2	SP1582217	Seatframe 545mm, cpl., pre set up to SW 43cm		1
3	SP1571810	Sealing strap with plugs		1
4	SP1571811	Groove stone with mounting parts		1
5	SP1555400	Adjustment Tube		1
6	SP1588443	Seat Rails 510mm (2pcs kit)		1
7	SP1588444	Seat Rails 545mm (2pcs kit)		1

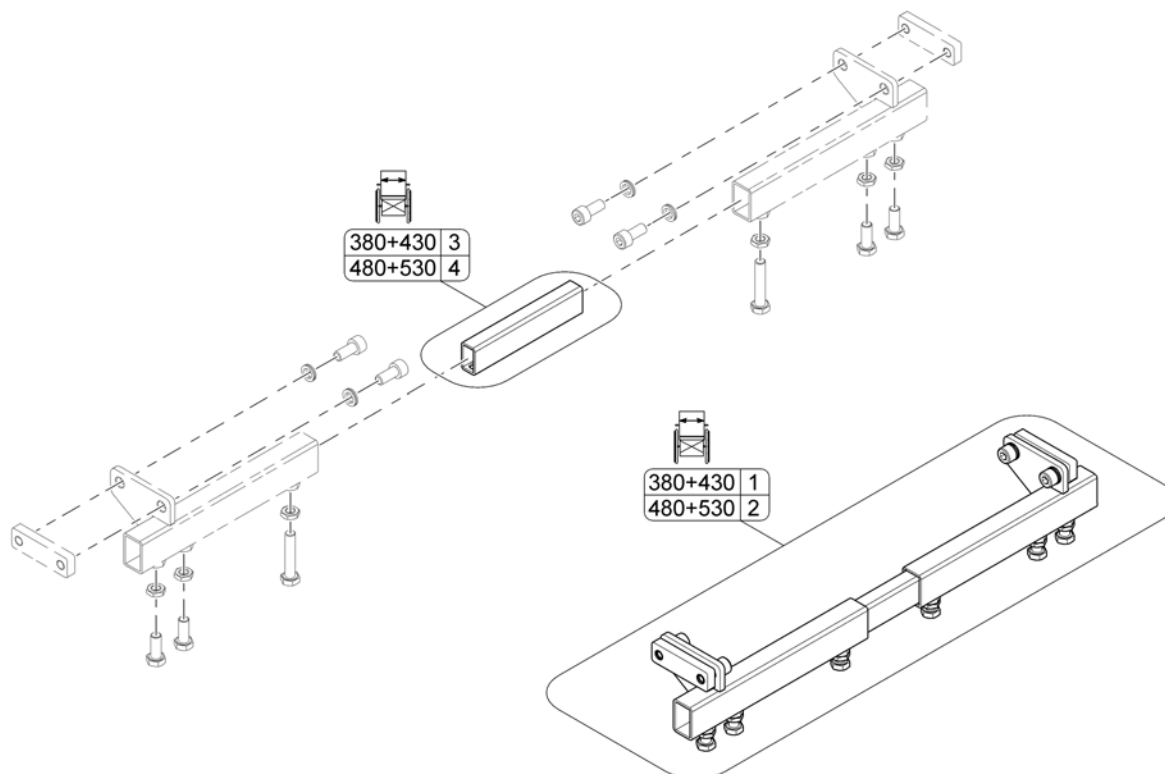


Pos.	Part number	Description	Valid from → until	Qty
1	SP1559305	Legrest Support SW380+430mm, cpl.		1
2	SP1559307	Legrest Support SW480+530mm, cpl.		1
3	SP1558134	Connection Tube SW380+430mm		1
4	SP1558135	Connection Tube SW480+530mm		1

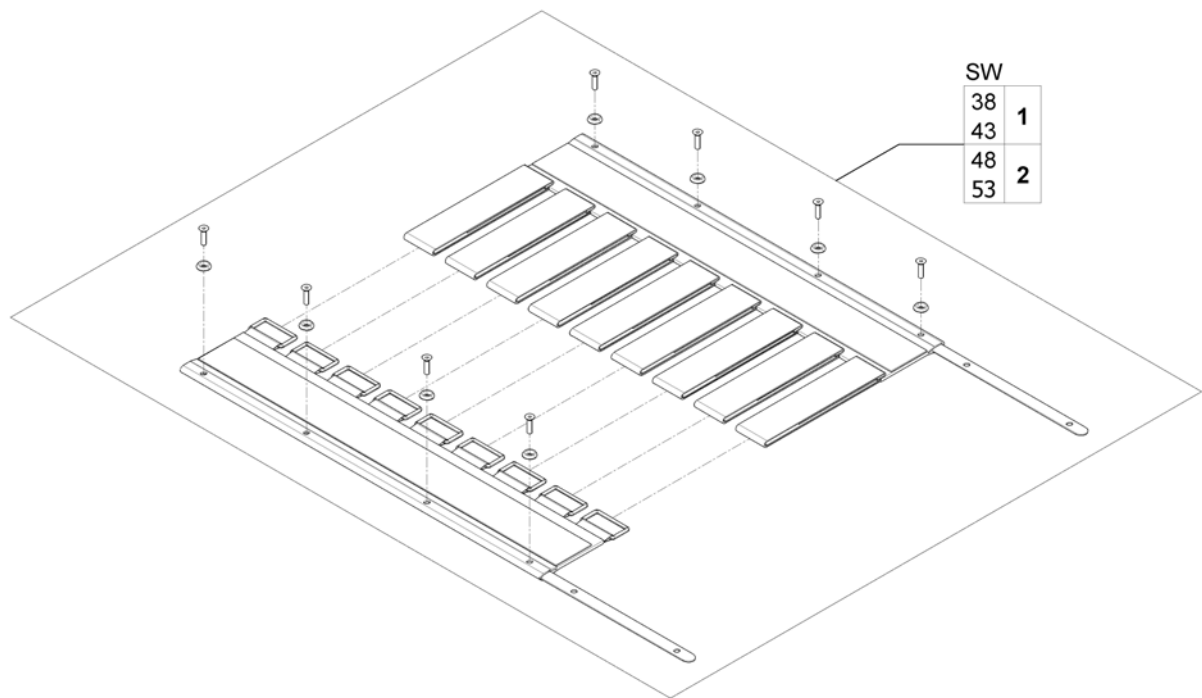

Pos.	Part number	Description	Valid from → until	Qty
1	SP1557877	Sling Seat SW380/430, 510mm		1
2	SP1555911	Sling Seat SW480/530, 510mm		1

Telescopic Seat Frame and Universal Adapter


1581886.iso

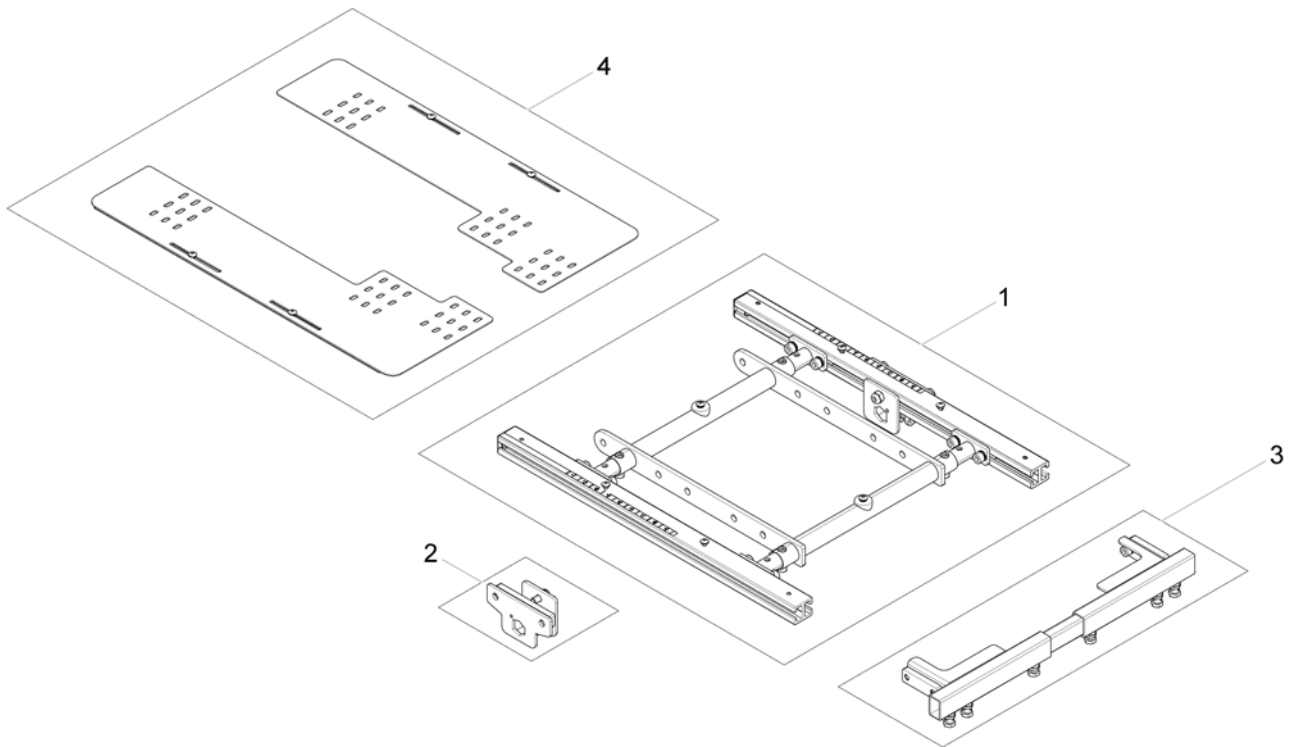

Pos.	Part number	Description	Valid from → until	Qty
1	-	Telescopic Seat Frame		1
2	-	Armrest bracket		1
3	-	Legrest Support		1
4	-	Adapter Kit for Universal Seat		1

Adapter Kit for Universal Seat


1581863.iso

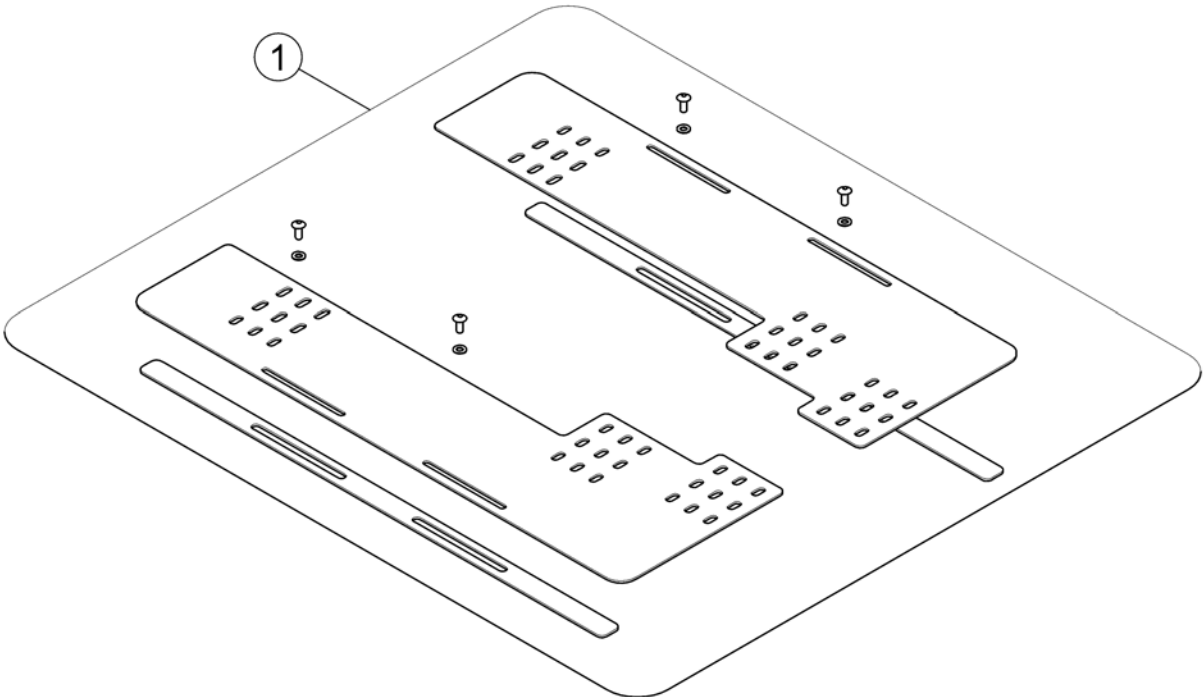

Pos.	Part number	Description	Valid from → until	Qty
1	SP1580750	Adapter Kit for Universal Seat 510mm		1

Seat Cushions "Standard" & "Matrx Flo-Tech"

1600754 iso

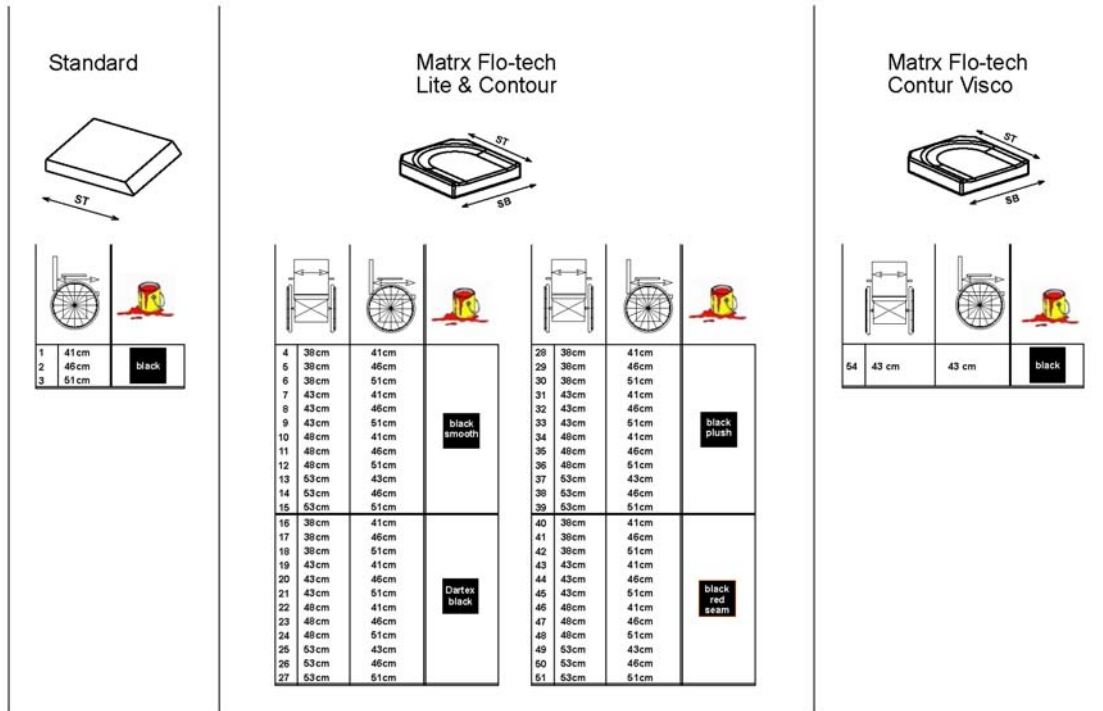

Pos.	Part number	Description	Valid from → until	Qty
1	1420460	Seat Cushion Standard - SW 430mm, SD 410mm <i>[2141]</i>		1
2	1420461	Seat Cushion Standard - SW 430mm, SD 460mm <i>[2141]</i>		1
3	1420462	Seat Cushion Standard - SW 430mm, SD 510mm <i>[2141]</i>		1
4	SPFLB15/16	Flo-Tech Lite 15x16" <i>[2142+2151]</i>		1
5	SPFLB15/18	Seatcushion Easy-Adapt SB38 x ST46cm black smooth <i>[2142+2151]</i>		1
6	SPFLB15/20	Seatcushion Easy-Adapt SB38 x ST51cm black smooth <i>[2142+2151]</i>		1
7	SPFLB17/16	Seatcushion Easy-Adapt SB43 x ST41cm black smooth <i>[2142+2151]</i>		1
8	SPFLB17/18	Seatcushion Easy-Adapt SB43 x ST46cm black smooth <i>[2142+2151]</i>		1
9	SPFLB17/20	Seatcushion Easy-Adapt SB43 x ST51cm black smooth <i>[2142+2151]</i>		1
10	SPFLB19/16	Seatcushion Easy-Adapt SB48 x ST41cm black smooth <i>[2142+2151]</i>		1
11	SPFLB19/18	Seatcushion Easy-Adapt SB48 x ST46cm black smooth <i>[2142+2151]</i>		1
12	SPFLB19/20	Flo-Tech Lite 19x20" <i>[2142+2151]</i>		1
13	SPFCB21/17	Flo-Tech Contour 21x17" <i>[2142+2151]</i>		1
14	SPFCB21/18	Flo-Tech Contour 21x18" <i>[2142+2151]</i>		1
15	SPFCB21/20	Flo-Tech Contour 21x20" <i>[2142+2151]</i>		1
16	SP1529546-TR26	seat cushion Flo-tech Lite SW 38 x SD 41 Dartex black <i>[2142+2152]</i>		1
17	SP1529547-TR26	seat cushion Flo-tech Lite SW 38 x SD 46 Dartex black <i>[2142+2152]</i>		1
18	SP1529548-TR26	seat cushion Flo-tech Lite SW 38 x SD 51 Dartex black <i>[2142+2152]</i>		1

Seat Cushions "Standard" & "Matrx Flo-Tech"


Pos.	Part number	Description	Valid from → until	Qty
19	SP1529549-TR26	seat cushion Flo-tech Lite SW 43 x SD 41 Dartex black <i>[2142+2152]</i>		1
20	SP1529550-TR26	seat cushion Flo-tech Lite SW 43 x SD 46 Dartex black <i>[2142+2152]</i>		1
21	SP1529551-TR26	seat cushion Flo-tech Lite SW 43 x SD 51 Dartex black <i>[2142+2152]</i>		1
22	SP1529552-TR26	seat cushion Flo-tech Lite SW 48 x SD 41 Dartex black <i>[2142+2152]</i>		1
23	SP1529553-TR26	seat cushion Flo-tech Lite SW 48 x SD 46 Dartex black <i>[2142+2152]</i>		1
24	SP1529554-TR26	seat cushion Flo-tech Lite SW 48 x SD 51 Dartex black <i>[2142+2152]</i>		1
25	SP1529555-TR26	seat cushion Flo-tech Contour SW 53 x SD 43 Dartex black <i>[2142+2152]</i>		1
26	SP1529556-TR26	seat cushion Flo-tech Contour SW 53 x SD 46 Dartex black <i>[2142+2152]</i>		1
27	SP1529570-TR26	seat cushion Flo-tech Contour SW 53 x SD 51 Dartex black <i>[2142+2152]</i>		1
28	SP1529546-TR35	seat cushion Flo-tech Lite SW 38 x SD 41 black chenille fabric <i>[2142+2154]</i>		1
29	SP1529547-TR35	seat cushion Flo-tech Lite SW 38 x SD 46 black chenille fabric <i>[2142+2154]</i>		1
30	SP1529548-TR35	seat cushion Flo-tech Lite SW 38 x SD 51 black chenille fabric <i>[2142+2154]</i>		1
31	SP1529549-TR35	seat cushion Flo-tech Lite SW 43 x SD 41 black chenille fabric <i>[2142+2154]</i>		1
32	SP1529550-TR35	seat cushion Flo-tech Lite SW 43 x SD 46 black chenille fabric <i>[2142+2154]</i>		1
33	SP1529551-TR35	seat cushion Flo-tech Lite SW 43 x SD 51 black chenille fabric <i>[2142+2154]</i>		1
34	SP1529552-TR35	Seat cushion Flo-tech Lite SW 48 x SD 41 black chenille fabric <i>[2142+2154]</i>		1
35	SP1529553-TR35	seat cushion Flo-tech Lite SW 48 x SD 46 black chenille fabric <i>[2142+2154]</i>		1
36	SP1529554-TR35	seat cushion Flo-tech Lite SW 48 x SD 51 black chenille fabric <i>[2142+2154]</i>		1
37	SP1529555-TR35	seat cushion Flo-tech Contour SW 53 X SD 43 black chenille fabric <i>[2142+2154]</i>		1
38	SP1529556-TR35	seat cushion Flo-tech Contour SW 53 x SD 46 black chenille fabric <i>[2142+2154]</i>		1
39	SP1529570-TR35	seat cushion Flo-tech Contour SW 53 x SD 51 black chenille fabric <i>[2142+2154]</i>		1
40	SP1529546	Seatcushion Easy-Adapt SB38 x ST41cm black with red seam <i>[2142+2153]</i>		1
41	SP1529547	Seatcushion Easy-Adapt SB38 x ST46cm black with red seam <i>[2142+2153]</i>		1
42	SP1529548	Seatcushion Easy-Adapt SB38 x ST51cm black with red seam <i>[2142+2153]</i>		1
43	SP1529549	Seatcushion Easy-Adapt SB43 x ST41cm black with red seam <i>[2142+2153]</i>		1
44	SP1529550	Seatcushion Easy-Adapt SB43 x ST46cm black with red seam <i>[2142+2153]</i>		1
45	SP1529551	Seatcushion Easy-Adapt SB43 x ST51cm black with red seam <i>[2142+2153]</i>		1
46	SP1529552	Seatcushion Easy-Adapt SB48 x ST41cm black with red seam <i>[2142+2153]</i>		1
47	SP1529553	Seatcushion Easy-Adapt SB48 x ST46cm black with red seam <i>[2142+2153]</i>		1
48	SP1529554	Seatcushion Easy-Adapt SB48 x ST51cm black with red seam <i>[2142+2153]</i>		1
49	SP1529555	Seatcushion Easy-Adapt SB53 x ST43cm black with red seam <i>[2142+2153]</i>		1
50	SP1529556	Seatcushion Easy-Adapt SB53 x ST46cm black with red seam <i>[2142+2153]</i>		1

Seat Cushions "Standard" & "Matrix Flo-Tech"



Pos.	Part number	Description	Valid from → until	Qty
51	SP1529570	Seatcushion Easy-Adapt SB53 x ST51cm black with red seam <i>[2142+2153]</i>		1
54	SP1501113	Seat Cushion Flo-tech Contour Visco SW 43cm x SD 43cm <i>[2143]</i>		1

Seat Cushions "Matrx Libra"

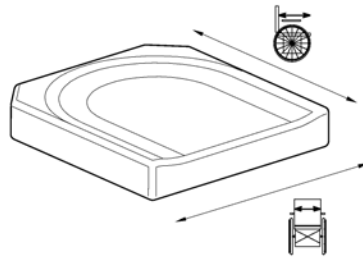1603356.iss




Matrx Libra

		 mm			
		380	430	480	530
 mm	410	1	4	7	-
	450	2	5	8	10
	510	3	6	9	11
	560	-	-	-	12



Pos.	Part number	Description	Valid from → until	Qty
1	LC1516	Matrx Libra cushions 380x410 mm		1
2	LC1518	Matrx Libra cushions 380x460 mm		1
3	LC1520	Matrx Libra cushions 380x510 mm		1
4	LC1716	Matrx Libra cushions 430x410 mm		1
5	LC1718	Matrx Libra cushions 430x460 mm		1
6	LC1720	Matrx Libra cushions 430x510 mm		1
7	LC1916	Matrx Libra cushions 480x410 mm		1
8	LC1918	Matrx Libra cushions 480x460 mm		1
9	LC1920	Matrx Libra cushions 480x510 mm		1
10	LC2118	Seat Cushion Matrx Libra 21x18"		1
11	LC2120	Seat Cushion Matrx Libra 21x20"		1
12	LC2122	Seat Cushion Matrx Libra 21x22"		1


Seat Cushions "Matrx Vi"

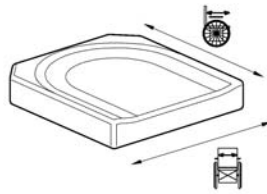1606752.iss



Matrx Vi

			 mm		
		380	430	480	530
410	1	4	7	-	
 mm	460	2	5	8	10
	510	3	6	9	11

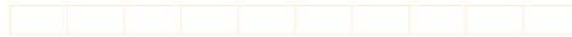


Pos.	Part number	Description	Valid from → until	Qty
1	SP1579786	Matrx Vi 16x16"		1
2	SP1579787	Matrx Vi 16x18"		1
3	SP1579788	Matrx Vi 16x20"		1
4	SP1579920	Matrx Vi 18x16"		1
5	SP1579921	Matrx Vi 18x18"		1
6	SP1579922	Matrx Vi 18x20"		1
7	SP1579923	Matrx Vi 20x16"		1
8	SP1579924	Matrx Vi 20x18"		1
9	SP1579925	Matrx Vi 20x20"		1
10	SP1579926	Matrx Vi 22x18"		1
11	SP1579927	Matrx Vi 22x20"		1

Ultra Low Maxx Seat

Pos.	Part number	Description	Valid from → until	Qty
-		Seat Column		1
-		Seat Tilt		1
-		Seat Lifter with Tilt		1
-		Posterior-Tilt		1
-		Seat Plates & Side Rails		1
-		Seat Cushions "Matrx Vi"		1
-		Seat Cushions "Matrx Flo-Tech"		1
-		Seat Cushions "Matrx Libra"		1

Seat Column


1602000.kim

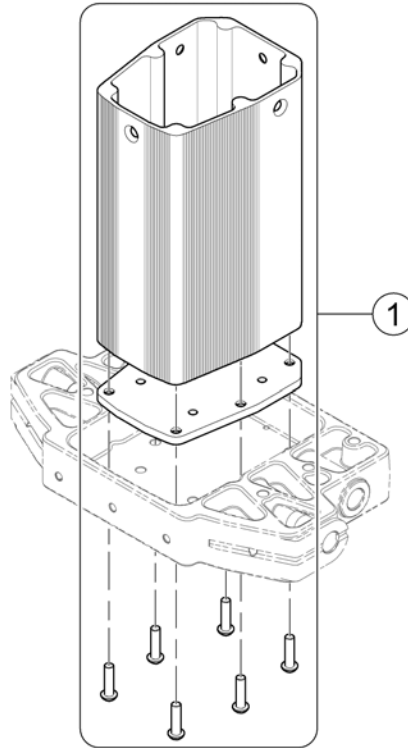

Pos.	Part number	Description	Valid from → until	Qty
1	SP1589681	Seat column, cpl.		1

Seat Tilt


1602021.kim

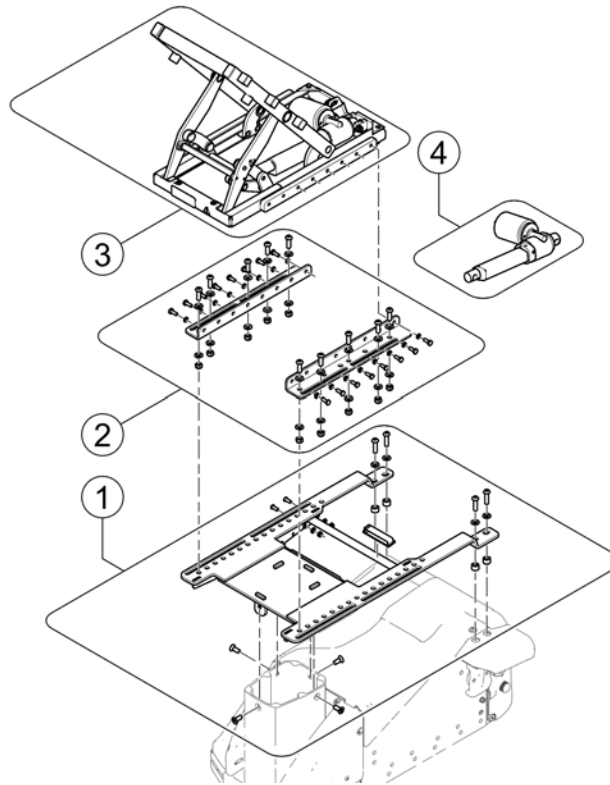


Pos.	Part number	Description	Valid from → until	Qty
1	SP1589710	Seat Mounting Adapter		1
2	SP1589711	Seat Adapter Rails		1
3	SP1585523	Tilt Module, cpl.		1
4	SP1588440	Tilt Actuator, cpl. with hardware		1
-	SPTRE0755	Cable, Tilt Actuator		1

Seat Lifter with Tilt

1602022.kim

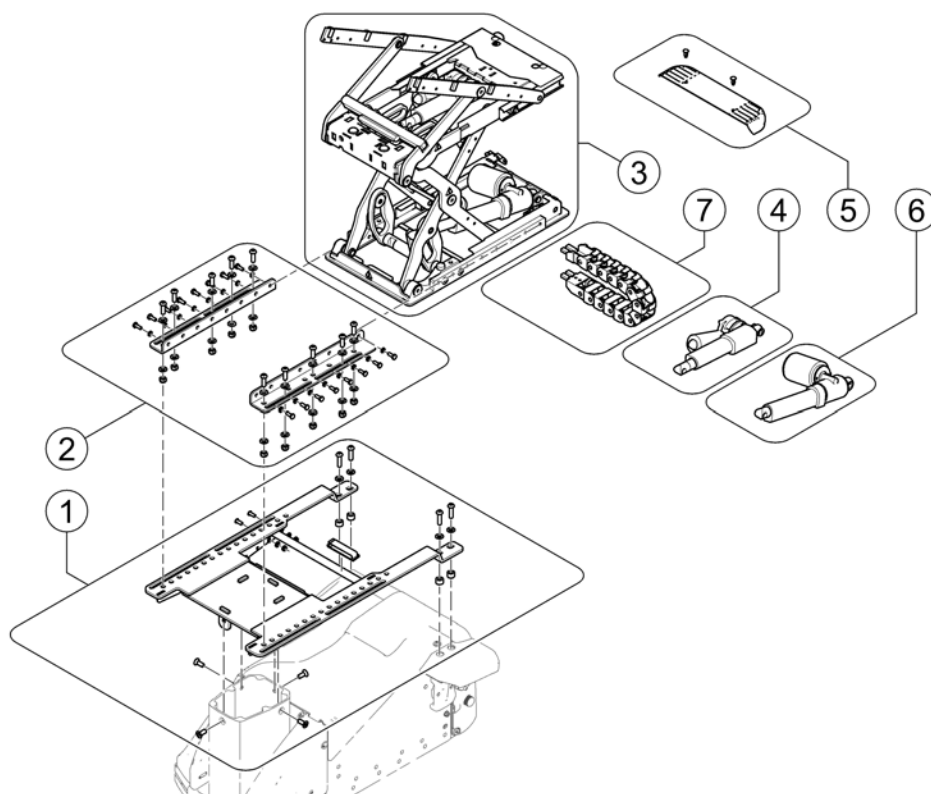


Pos.	Part number	Description	Valid from → until	Qty
1	SP1589710	Seat Mounting Adapter		1
2	SP1589711	Seat Adapter Rails		1
3	SP1585524	Lifter/Tilt Module, cpl.		1
4	SP1588442	Lifter actuator, cpl. with hardware		1
5	SPTRC2071	Cover, Rear Wiring Lifter, cpl.		1
6	SP1588441	Tilt actuator, cpl. with hardware		1
7	SP1604199	Energie chain		1

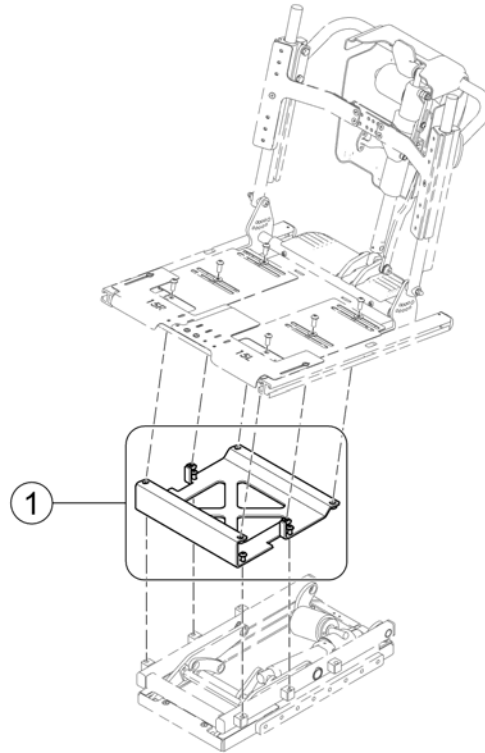

Pos.	Part number	Description	Valid from → until	Qty
1	SPTRC1923	Pre-Tilt Mounting Kit 5° <i>Posterior-Tilt</i>		1

Seat Plates & Side Rails


1588152 Rev

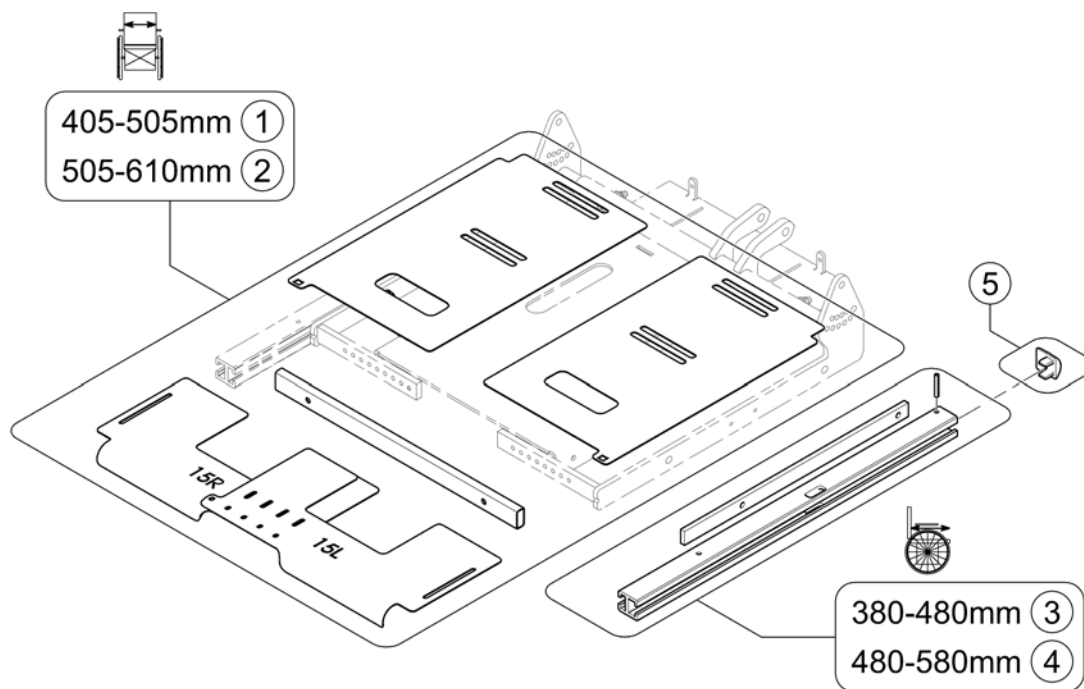

Pos.	Part number	Description	Valid from → until	Qty
1	SPTRC2187	Seat Pans - SW 405-505mm (16-20")		1
2	SPTRC2189	Seat Pans - SW 505-610mm (19-24")		1
3	SPTRC2141	Seat Rail Kit SD 380-480mm, cpl.		1
4	SPTRC2142	Seat Rail Kit SD 480-580mm, cpl.		1
5	SPTRC2090	End cap kit (2 pcs)		1

Seat Cushions "Matrx Vi"

1588153.iss

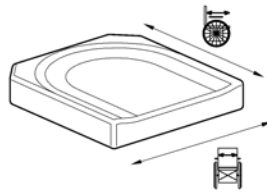


Matrx Vi


		mm								
		405	430	455	480	505	530	555	580	610
mm	380	1	4	4	8	8	-	-	-	-
	405	1	4	4	8	8	-	-	-	-
	430	1	4	4	8	8	-	-	-	-
	455	2	5	5	9	9	12	12	15	15
	480	2	5	5	9	9	12	12	15	15
	505	3	6	6	10	10	13	13	16	16
	530	3	6	6	10	10	13	13	16	16
	555	-	7	7	11	11	14	14	17	17
	580	-	7	7	11	11	14	14	17	17


Pos.	Part number	Description	Valid from → until	Qty
1	SP1579786	Matrx Vi 16x16"		1
2	SP1579787	Matrx Vi 16x18"		1
3	SP1579788	Matrx Vi 16x20"		1
4	SP1579920	Matrx Vi 18x16"		1
5	SP1579921	Matrx Vi 18x18"		1
6	SP1579922	Matrx Vi 18x20"		1
7	SP1584183	Matrx Vi 18x22"		1
8	SP1579923	Matrx Vi 20x16"		1
9	SP1579924	Matrx Vi 20x18"		1
10	SP1579925	Matrx Vi 20x20"		1
11	SP1584184	Matrx Vi 20x22"		1
12	SP1579926	Matrx Vi 22x18"		1
13	SP1579927	Matrx Vi 22x20"		1
14	SP1584247	Matrx Vi 22x22"		1
15	SP1584384	Matrx Vi 24x18"		1
16	SP1584417	Matrx Vi 24x20"		1
17	SP1584418	Matrx Vi 24x22"		1

Seat Cushions "Matrx Flo-Tech"

1588508.iss

						
		405	430	455	480	505
Matrx Flo-Tech Lite	380	1	1	5	5	9
	405	2	2	6	6	9
	430	2	2	6	6	10
	455	3	3	7	7	10
	480	3	3	7	7	-
	505	4	4	8	8	-
	530	4	4	8	8	-

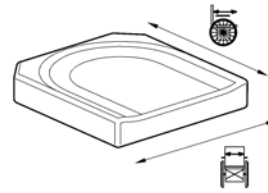


											
		405	430	455	480	505	530	555	580	610	
Matrx Flo-Tech Contour	380	11	11	15	15	19	-	-	-	-	
	405	12	12	16	16	19	-	-	-	-	
	430	12	12	16	16	20	20	24	24	27	
	455	13	13	17	17	21	21	24	24	27	
	480	13	13	17	17	21	21	25	25	28	
	505	14	14	18	18	22	22	25	25	28	
	530	14	14	18	18	22	22	26	26	29	
	555	-	-	-	-	23	23	26	26	29	



Pos.	Part number	Description	Valid from → until	Qty
1	FLB17/15	Flo-Tech Lite 17X15"		1
2	FLB17/16	Flo-Tech Lite 17X16"		1
3	FLB17/18	Flo-Tech Lite 17X18"		1
4	FLB17/20	Flo-Tech Lite 17X20"		1
5	FLB19/15	Flo-Tech Lite 19x15"		1
6	FLB19/16	Flo-Tech Lite 19x16"		1
7	FLB19/18	Flo-Tech Lite 19x18"		1
8	SPFLB19/20	Flo-Tech Lite 19x20"		1
9	SP1584714	Flo-Tech Lite 20x15"		1
10	SP1584713	Flo-Tech Lite 20x17"		1
11	FCB17/15	Flo-Tech Contour 17x15"		1
12	SPFCB17_16	Flo-Tech Contour 17x16"		1
13	SPFCB17_18	Flo-Tech Contour 17x18"		1
14	SPFCB17_20	Flo-Tech Contour 17x20"		1
15	SPFCB19_15	Flo-Tech Contour 19x15"		1
16	SPFCB19_16	Flo-Tech Contour 19x16"		1
17	SPFCB19_18	Flo-Tech Contour 19x18"		1
18	SPFCB19_20	Flo-Tech Contour 19x20"		1
19	SPFCB20_15	Flo-Tech Contour 20x15"		1
20	SPFCB21/17	Flo-Tech Contour 21x17"		1
21	SPFCB21/18	Flo-Tech Contour 21x18"		1
22	SPFCB21/20	Flo-Tech Contour 21x20"		1
23	SP1584739	Flo-Tech Contour 21x22"		1
24	SP1584740	Flo-Tech Contour 23x17"		1
25	SP1584741	Flo-Tech Contour 23x19"		1
26	SP1584742	Flo-Tech Contour 23x21"		1
27	SP1584743	Flo-Tech Contour 24x17"		1
28	SP1584770	Flo-Tech Contour 24x19"		1
29	SP1584771	Flo-Tech Contour 24x21"		1

Seat Cushions "Matrx Libra"

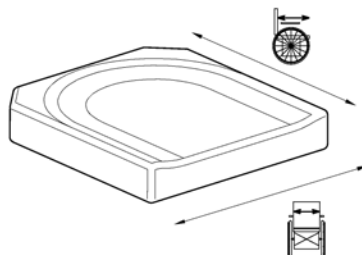1603355.iss




Matrx Libra

		 mm						
		405	430	455	480	505	530	555
 mm	405	1	4	7	10	13	-	-
	430	1	4	7	10	13	-	-
	455	2	5	8	11	14	16	19
	480	2	5	8	11	14	16	19
	505	3	6	9	12	15	17	20
	530	3	6	9	12	15	17	20
	555	-	-	-	-	-	18	21
	580	-	-	-	-	-	18	21


Pos.	Part number	Description	Valid from → until	Qty
1	LC1616	Matrx Libra cushions 410x410 mm		1
2	LC1618	Matrx Libra cushions 410x460 mm		1
3	LC1620	Matrx Libra cushions 410x510 mm		1
4	LC1716	Matrx Libra cushions 430x410 mm		1
5	LC1718	Matrx Libra cushions 430x460 mm		1
6	LC1720	Matrx Libra cushions 430x510 mm		1
7	LC1816	Matrx Libra cushions 460x410 mm		1
8	LC1818	Matrx Libra cushions 460x460 mm		1
9	LC1820	Matrx Libra cushions 380x510 mm		1
10	LC1916	Matrx Libra cushions 480x410 mm		1
11	LC1918	Matrx Libra cushions 480x460 mm		1
12	LC1920	Matrx Libra cushions 480x510 mm		1
13	LC2016	Matrx Libra cushions 510x410 mm		1
14	LC2018	Matrx Libra cushions 510x460 mm		1
15	LC2020	Matrx Libra cushions 510x510 mm		1
16	LC2118	Seat Cushion Matrx Libra 21x18"		1
17	LC2120	Seat Cushion Matrx Libra 21x20"		1
18	LC2122	Seat Cushion Matrx Libra 21x22"		1
19	LC2218	Matrx Libra cushions 560x460 mm		1
20	LC2220	Matrx Libra cushions 560x510 mm		1
21	LC2222	Seat Cushion Matrx Libra 22x22"		1

Recaro Seat

1573131.iss

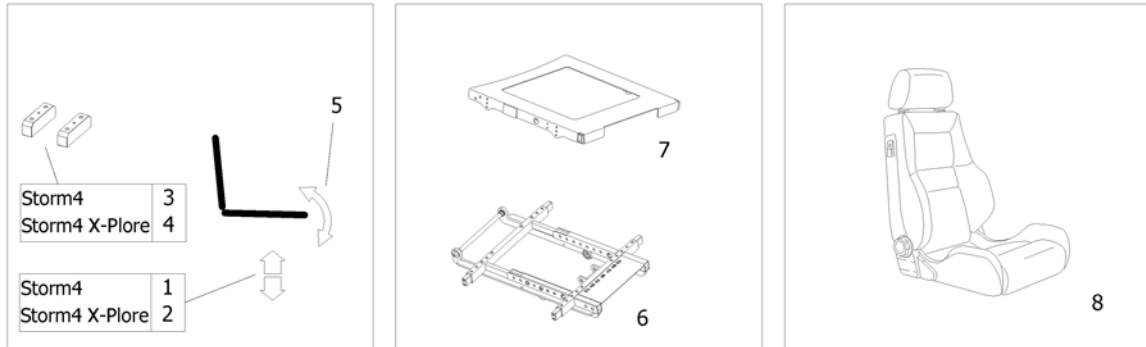


Pos.	Part number	Description	Valid from → until	Qty
1	-	Seat column / -raiser Storm4 <i>[1025/1026/1030/1040]</i>		1
2	-	Seat column / -raiser Storm4 X-Plore <i>[1025/1026/1030/1040]</i>		1
3	-	Seat Height Adapter Storm4 <i>[1040]</i>		1
4	-	Seat Height Adapter Storm4 X-Plore <i>[1040]</i>		1
5	-	Seat Tilt <i>[1002][1014]</i>		1
6	-	Seat Frame		1
7	-	Seat Plate		1
8	-	Recaro Seat		1

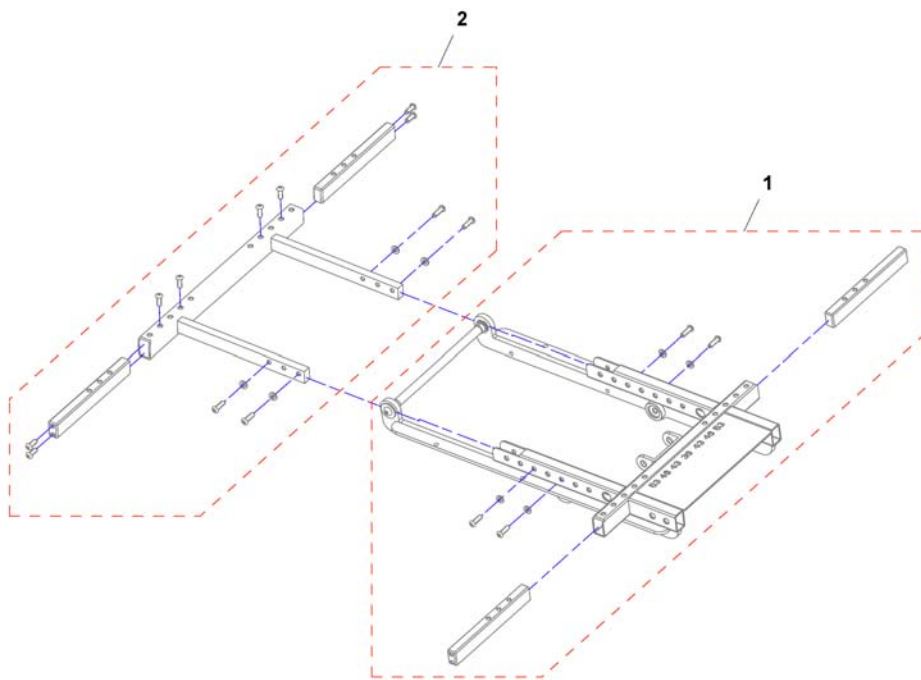

Pos.	Part number	Description	Valid from → until	Qty
1	SP1536856	Front Seat Frame		1
2	SP1542966	Rearframe for recaroseat		1

Seat Plate


1573206.iso

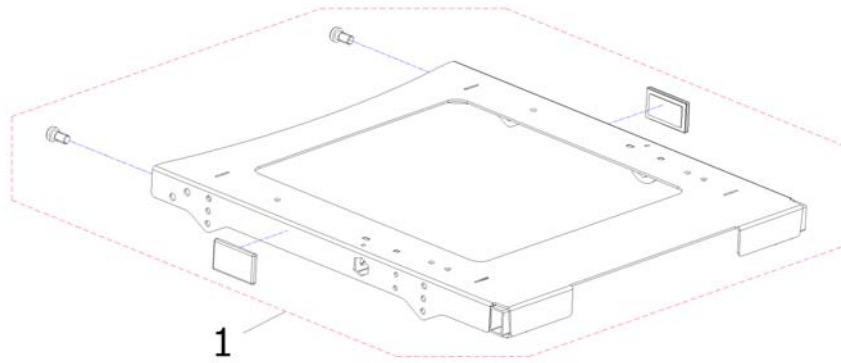

Pos.	Part number	Description	Valid from → until	Qty
1	SP1530519	Seatplate Recaro		1

Recaro Seat


1422546.dwg

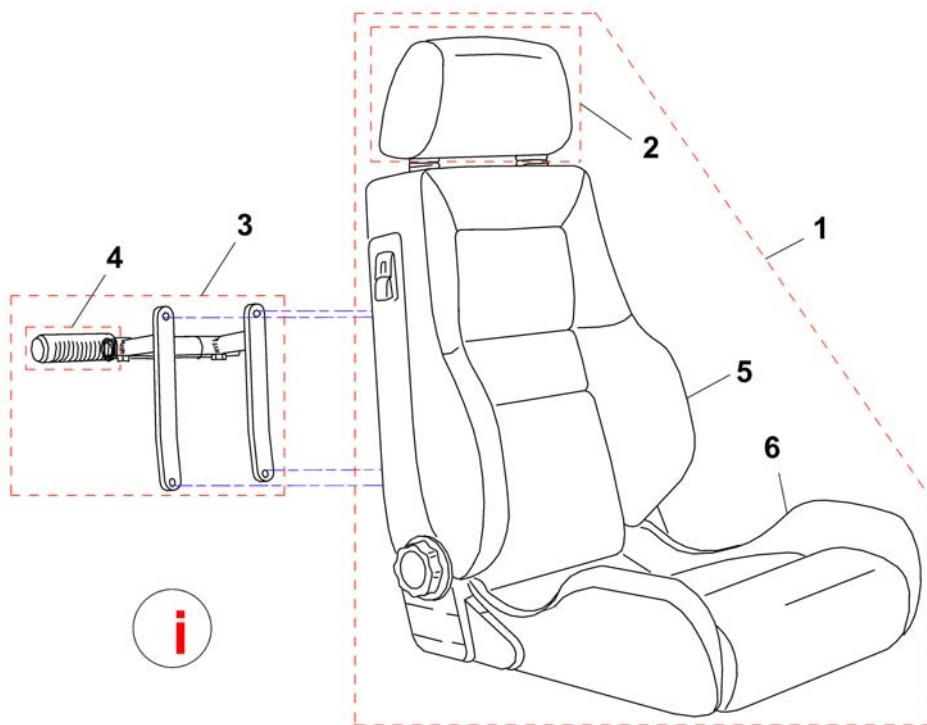

Pos.	Part number	Description	Valid from → until	Qty
i	-	Recaro Mobility 24V is no longer available. Please pay attention to the change list!		1
i	-	Change list Recaro 24V		1
	F367002	Belt with Velcro fastener for Recaro		1
	Z163001A	Belt with snap lock & Velcro fastner for Recaro Seat		1
	SP1423173	RECARO SEAT MOUNTING KIT		1
1	SP1552597	Recaroseat Mobilty, 12V, cpl.		1
1	SP1422546	Recaro Seat N-Joy cpl.		1
2	N89999	HEADREST - SPORT NARDO		1
3	SP1423177	HOLDER F. PUSH HANDLE RECARO CPL.		1
4	1435421	Handle plastic black Ø22 "Invacare"		1
5	SP1552596	Backrest, Recaro Mobility, 12v, cpl. <i>Recaro Mobility 24V is no longer available. Please pay attention to the compatibility chart!</i>		1
5	SP1552781	Backrest Recaro Mobility, mechanic		1
6	SP1419207	Seatpart Recaro Mobility		1

Optimist Seat


1573132.iss

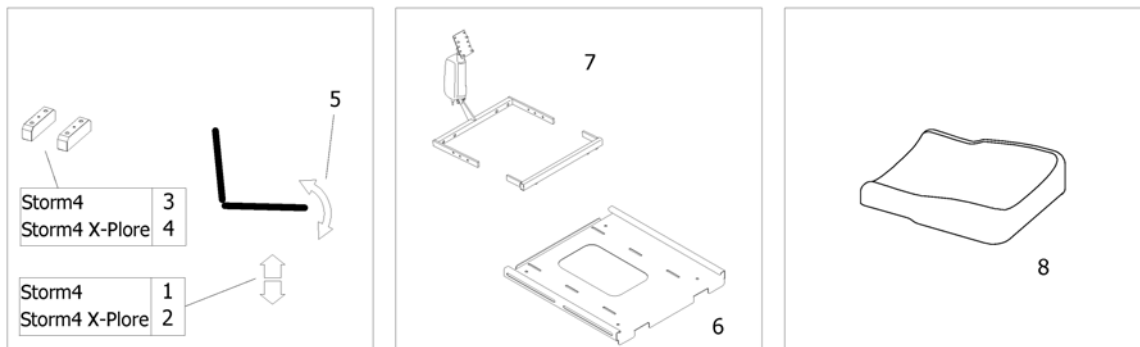

Pos.	Part number	Description	Valid from → until	Qty
1	-	Seat column / -raiser Storm4 <i>[1025/1026/1030/1040]</i>		1
2	-	Seat column / -raiser Storm4 X-Plore <i>[1025/1026/1030/1040]</i>		1
3	-	Seat Height Adapter Storm4 <i>[1040]</i>		1
4	-	Seat Height Adapter Storm4 X-Plore <i>[1040]</i>		1
5	-	Seat Tilt <i>[1002][1014]</i>		1
6	-	Seat Plate		1
7	-	Seat Frame		1
8	-	Seat Cushions Optimist Seat		1

Seat Plate

1573207.iso

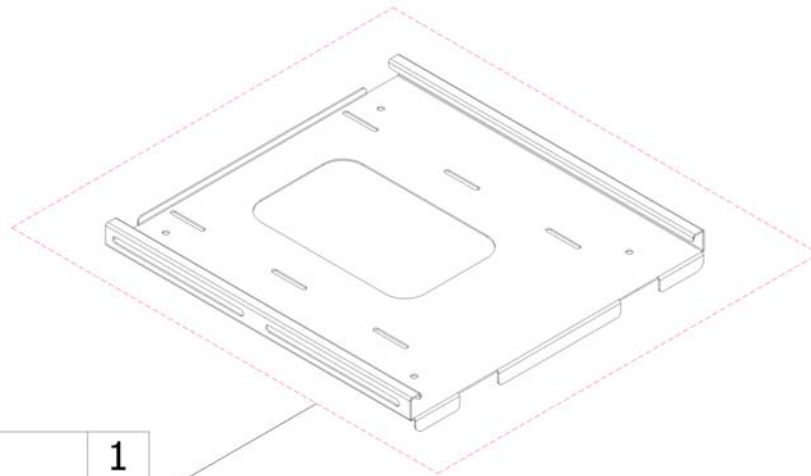


SW 38	1
SW 45	2
SW 50	3

Pos.	Part number	Description	Valid from → until	Qty
1	SP1524647	Seatplate Arne Jensen 38cm		1
2	SP1524643	Seatplate Arne Jensen 45cm		1
3	SP1524646	Seatplate Arne Jensen 50cm		1


Seat Frame

1573478.iso

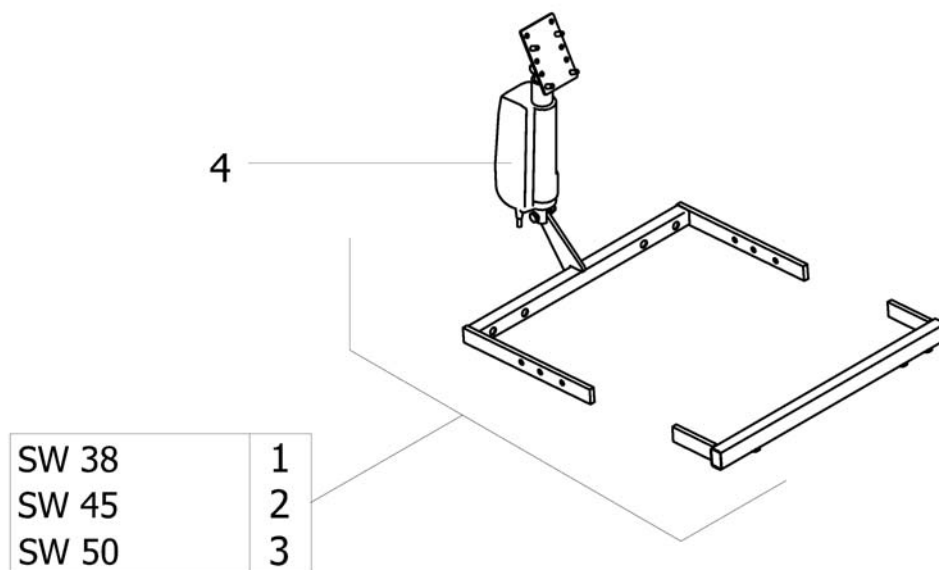

Pos.	Part number	Description	Valid from → until	Qty
1	SP1428490	Seat support Optimist Seat, SW 38cm		1
2	SP1428110	Seat support Optimist Seat, SW 45cm		1
3	SP1428491	Seat support Optimist Seat, SW 50cm		1
4	1419417	MOTOR LA 28.1-100-24-001		1


Seat Cushions Optimist Seat

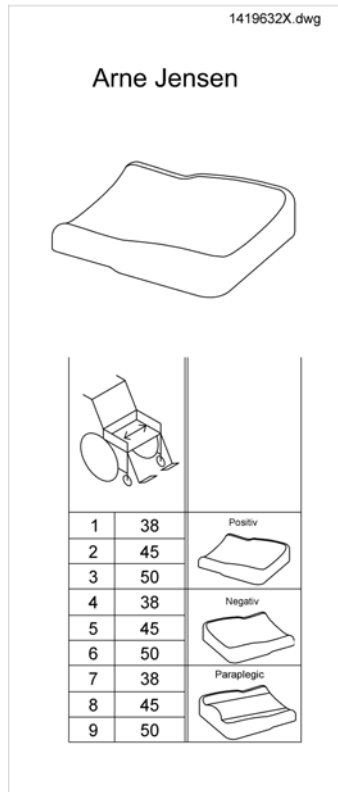

Pos.	Part number	Description	Valid from → until	Qty
1	1419632	Seat Cushion positive, SW 38cm, black		1
2	5313953	Seat Cushion positive, SW 45cm, black		1
3	5314038	Seat Cushion positive, SW 50cm, black		1
4	1419633	Seat Cushion negative, SW 38cm, black		1
5	5313958	Seat Cushion negative, SW 45cm, black		1
6	5314041	Seat Cushion negative, SW 50cm, black		1
7	1419634	Seat Cushion paraplegic, SW 38cm, black		1
8	5313962	Seat Cushion paraplegic, SW 45cm, black		1
9	5314044	Seat Cushion paraplegic, SW 50cm, black		1

Storm4 TTPlus


Pos.	Part number	Description	Valid from → until	Qty
1	-	EasyAdapt Seat <i>[1180]</i>		1
2	-	Recaro Seat <i>[1152]</i>		1
3	-	Optimist Seat <i>[1171]</i>		1
4	-	Modulite Seat <i>[2100]</i>		1

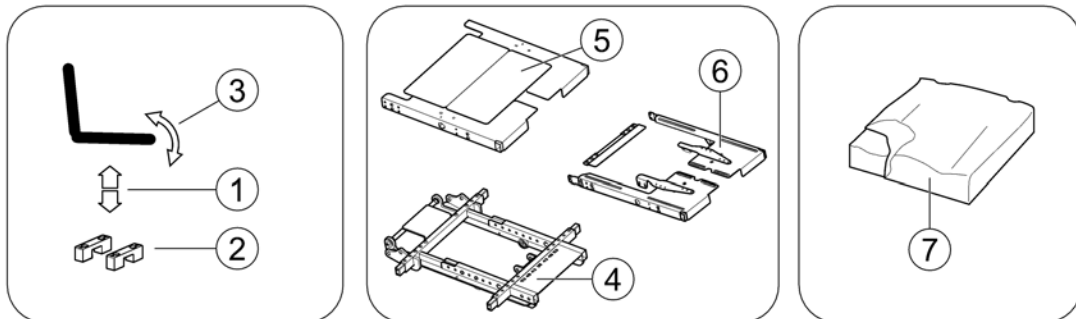

Pos.	Part number	Description	Valid from → until	Qty
1	-	Seat column / Seatriser [1025/1026/1030/1040]		1
2	-	Seat Height Adapter [1040]		1
3	-	Seat Tilt [1002][1014]		1
4	-	Seat Frame		1
5	-	Seat Plate		1
6	-	Adapter Kit for Universal Seat [1064]		1
7	-	Seat Cushion		1


Seat column / Seatriser

1583091.000

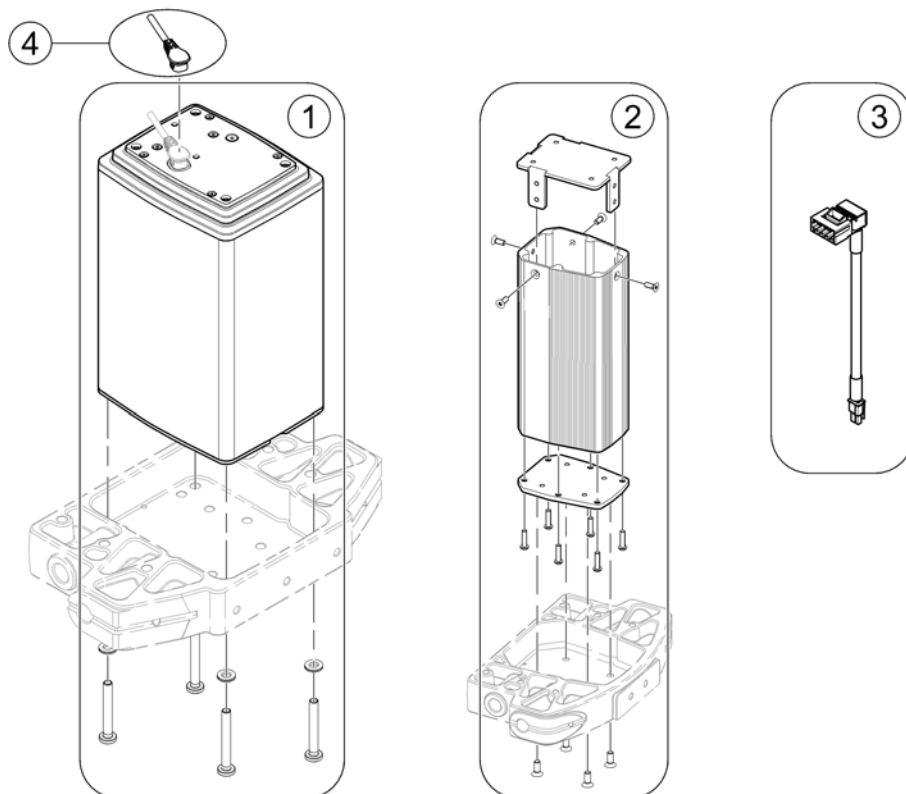

Pos.	Part number	Description	Valid from → until	Qty
1	SP1556263	Lifting column, cpl.		1
-	SP1568361	Mounting Kit lifter for seat bracket		1
2	SP1548536	Seat column, fixed, height 45cm and 48cm, adjustable		1
3	SP1579553	Slow Stop cable lifter GGB2		1
4	SP1579637	Lifter cable LINAK		1


1583151.000

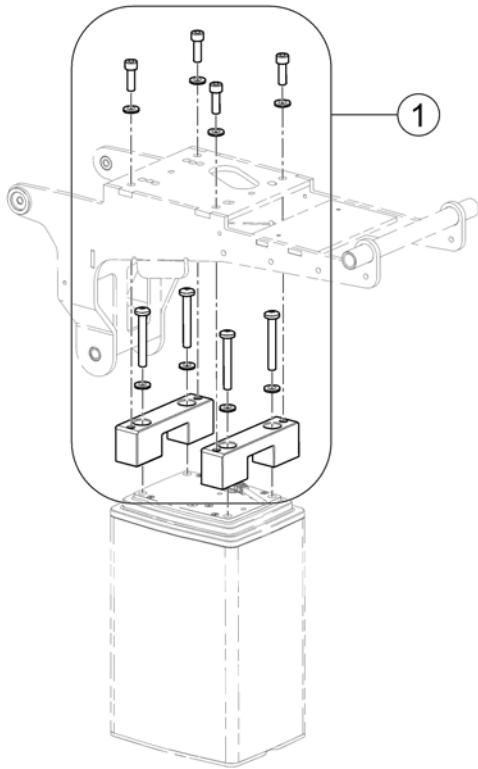

Pos.	Part number	Description	Valid from → until	Qty
1	SP1555071	Seatheightadapter cpl.		1

Seat Tilt


1503166.kso

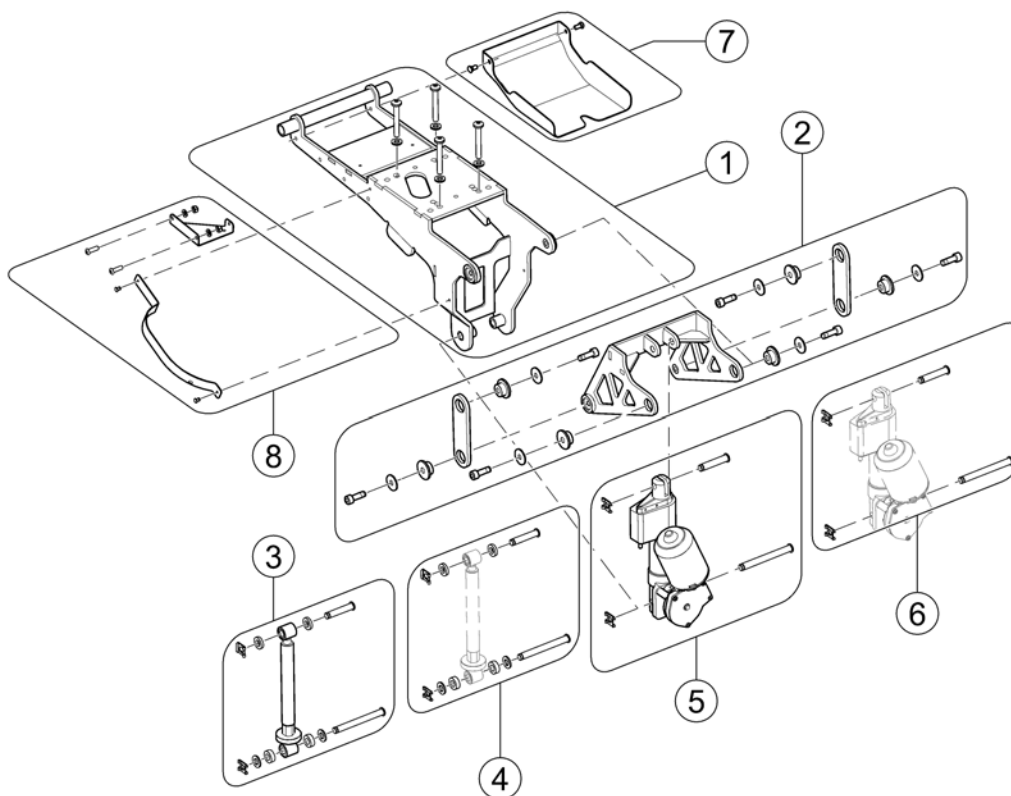

Pos.	Part number	Description	Valid from → until	Qty
1	SP1551991	Tilt carrier <i>[1002][1014]</i>		1
2	SP1559711	Tiltmechanism, cpl. <i>[1002][1014]</i>		1
3	SP1551010	Tilt spindle, cpl. <i>[1002]</i>		1
4	SP1551009	Mounting-Kit		1
5	SP1588045	Tilt Motor Reac 23.3 (incl. Hardware) <i>[1014]</i>		1
6	SP1532481	Mounting-Kit for powered tilt		1
7	SP1580197	Relais cover, cpl.		1
8	SP1603273	Cable Support Sheets, cpl. <i>[1030][1040]</i>		1

Recaro Seat


1583149.00

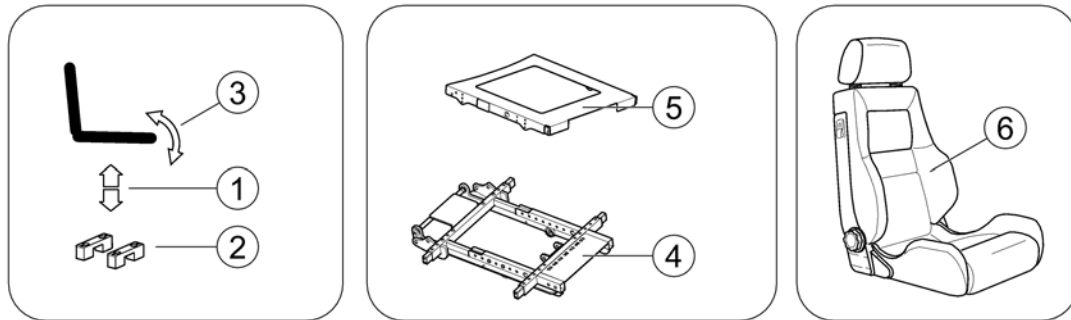

Pos.	Part number	Description	Valid from → until	Qty
1	-	Seat column / Seatrisher [1025/1026/1030/1040]		1
2	-	Seat Height Adapter [1040]		1
3	-	Seat Tilt [1002][1014]		1
4	-	Seat Frame		1
5	-	Seat Plate		1
6	-	Recaro Seat		1

Optimist Seat


1583150.000

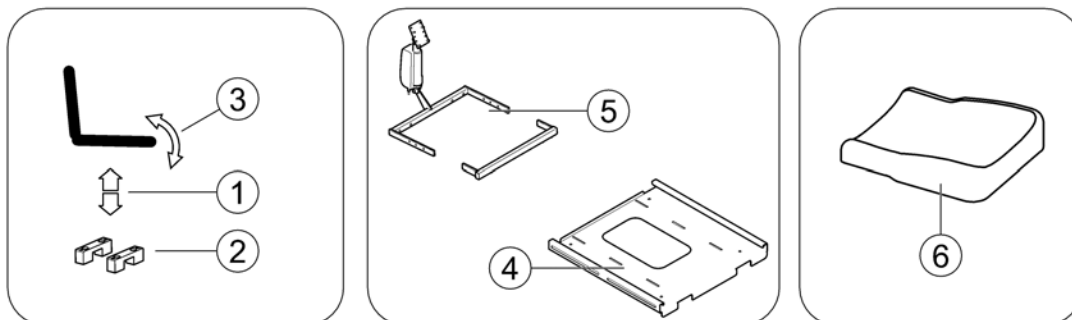

Pos.	Part number	Description	Valid from → until	Qty
1	-	Seat column / Seatriser [1025/1026/1030/1040]		1
2	-	Seat Height Adapter [1040]		1
3	-	Seat Tilt [1002][1014]		1
4	-	Seat Plate		1
5	-	Seat Frame		1
6	-	Seat Cushions Optimist Seat		1

Modulite Seat


1583148 Rev

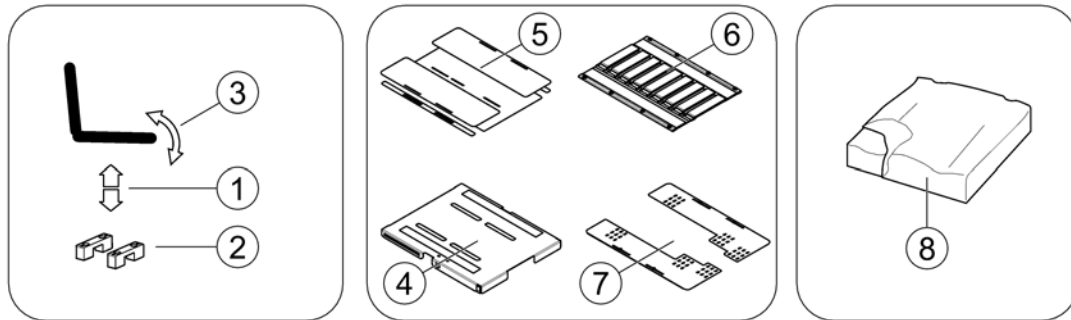

Pos.	Part number	Description	Valid from → until	Qty
1	-	Seat column / Seatriser <i>[1025/1026/1030/1040]</i>		1
2	-	Seat Height Adapter <i>[1040]</i>		1
3	-	Seat Tilt <i>[1002][1014]</i>		1
4	-	One Piece Seat Plate <i>[2110]</i>		1
5	-	Telescopic Seat Frame and Plate <i>[2111]</i>		1
6	-	Telescopic Seat Frame and Sling Seat <i>[2112]</i>		1
7	-	Telescopic Seat Frame and Universal Adapter <i>[2109]</i>		1
8	-	Seat Cushions "Standard" & "Matrx Flo-Tech" <i>[2141/2142/2143]</i>		1
8	-	Seat Cushions "Matrx Libra" <i>[2147]</i>		1
8	-	Seat Cushions "Matrx Vi" <i>[2144]</i>		1

Seat Tilt

1583169.kim

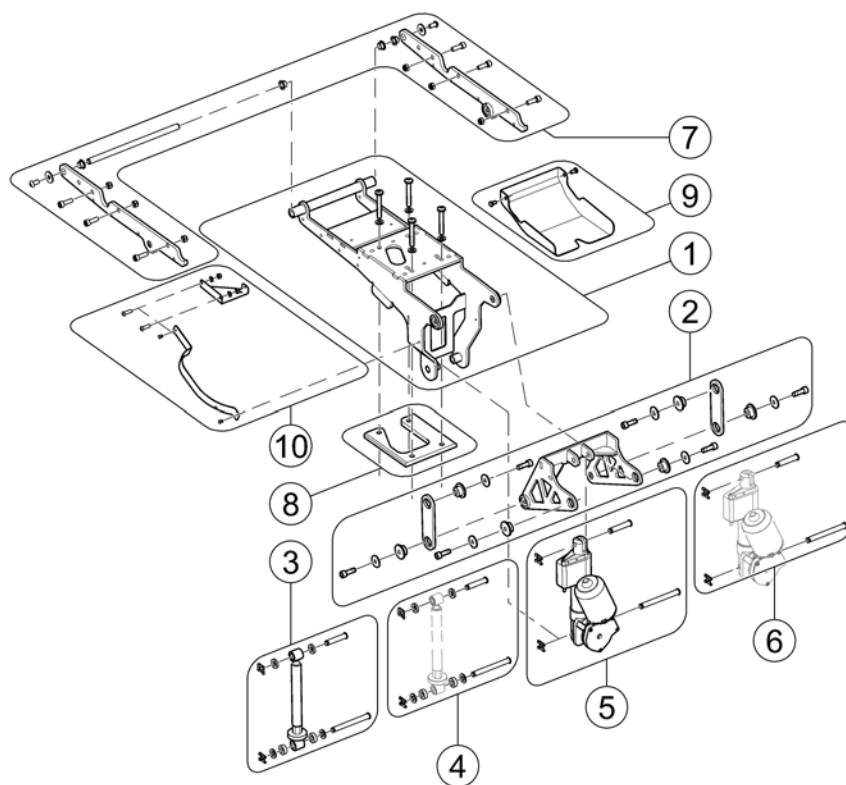

Pos.	Part number	Description	Valid from → until	Qty
1	SP1551991	Tilt carrier [1002][1014]		1
2	SP1559711	Tiltmechanism, cpl. [1002][1014]		1
3	SP1551010	Tilt spindle, cpl. [1002]		1
4	SP1551009	Mounting-Kit		1
5	SP1588045	Tilt Motor Reac 23.3 (incl. Hardware) [1014]		1
6	SP1532481	Mounting-Kit for powered tilt		1
7	SP1573217	Modulite Seat Support [2001]		1
8	SP1580196	Seat Height Adapter GGB2 [2001]+[1290][1275][1483]		1
9	SP1580197	Relais cover, cpl.		1
10	SP1603273	Cable Support Sheets, cpl. [1030][1040]		1

Backrest

ST4backrest.lin



Pos.	Part number	Description	Valid from → until	Qty
1	-	Storm4		0
2	-	Storm4 TTPlus		0

Storm4


Pos.	Part number	Description	Valid from → until	Qty
1	-	Backrest for Standard Seat		1
2	-	Backrest for EasyAdapt Seat		1
3	-	Backrest for Modulite Seat		1
4	-	Backrest for Ultra Low Maxx Seat		1
5	-	Backrest for Optimist Seat		1

Invacare® Storm4 | Backrest
Backrest for Standard Seat

1573228.iss




Pos.	Part number	Description	Valid from → until	Qty
1	-	Standard Backrest [1178/0970]		1

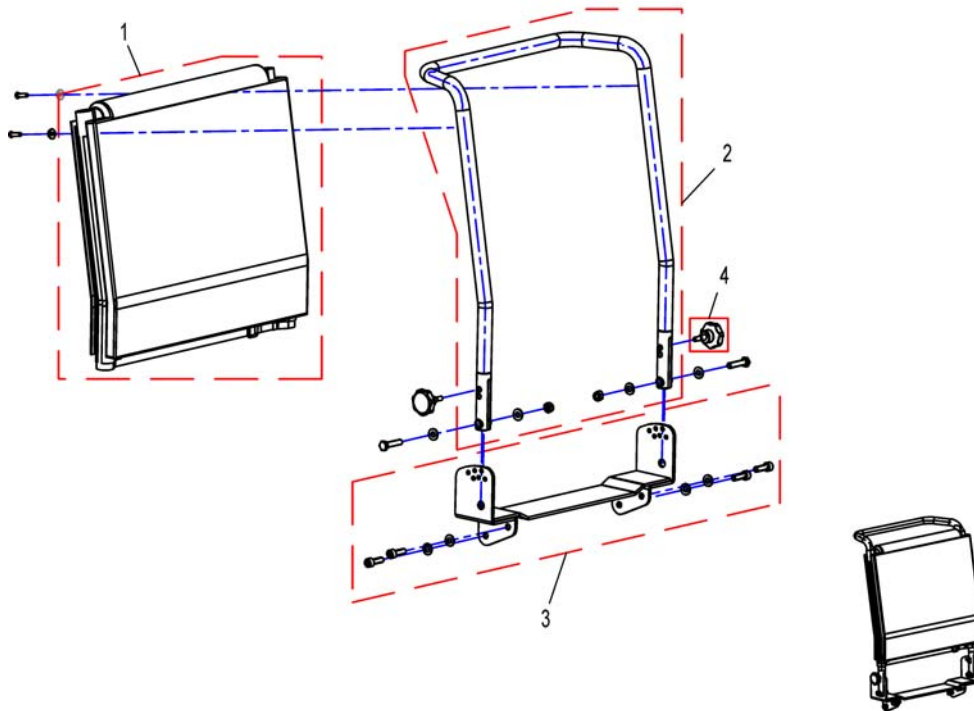

Pos.	Part number	Description	Valid from → until	Qty
1	1422062	Sling Back with Backrest upholstery 480 mm		1
1	1422063	Sling Back with Backrest upholstery 540mm		1
2	SP1419996	Backrest Frame 48cm height		1
2	SP1420465	Backrest Frame 54cm height		1
3	SP1530528	Bracket for standard backrest		1
4	SP1420444	Star Grip M8		1


Invacare® Storm4 | Backrest
Backrest for EasyAdapt Seat

1573229.iss

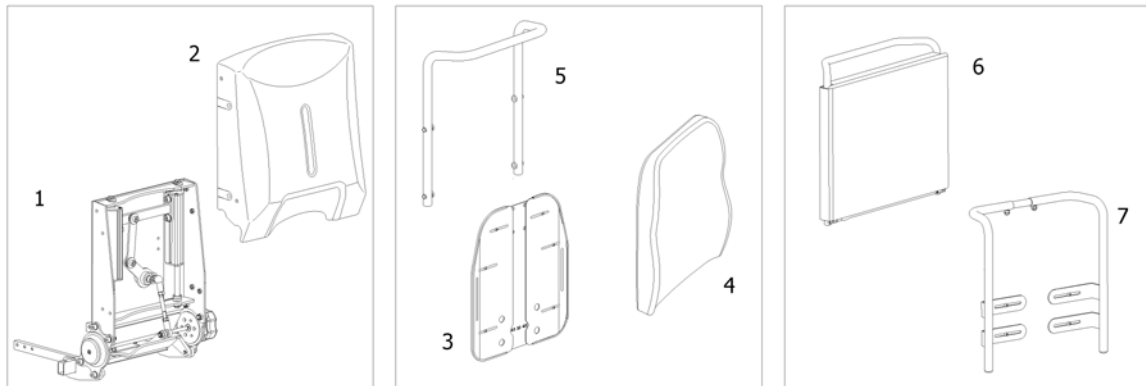


Pos.	Part number	Description	Valid from → until	Qty
1	-	Backrestadjusting		1
2	-	Backrestcover for EasyAdapt		1
3	-	Backplate EasyAdapt		1
3	-	Backplate EasyAdapt for DAHL Docking Station		1
4	-	Backrest Cushions [1985/1986/1987/1988]		1
5	-	Push Handle EasyAdapt [1307]		1
6	-	Tension adjustable Sling Back [1185/1186]		1
7	-	"Universal"-Backrest frame [0904]		1

Backrestadjusting

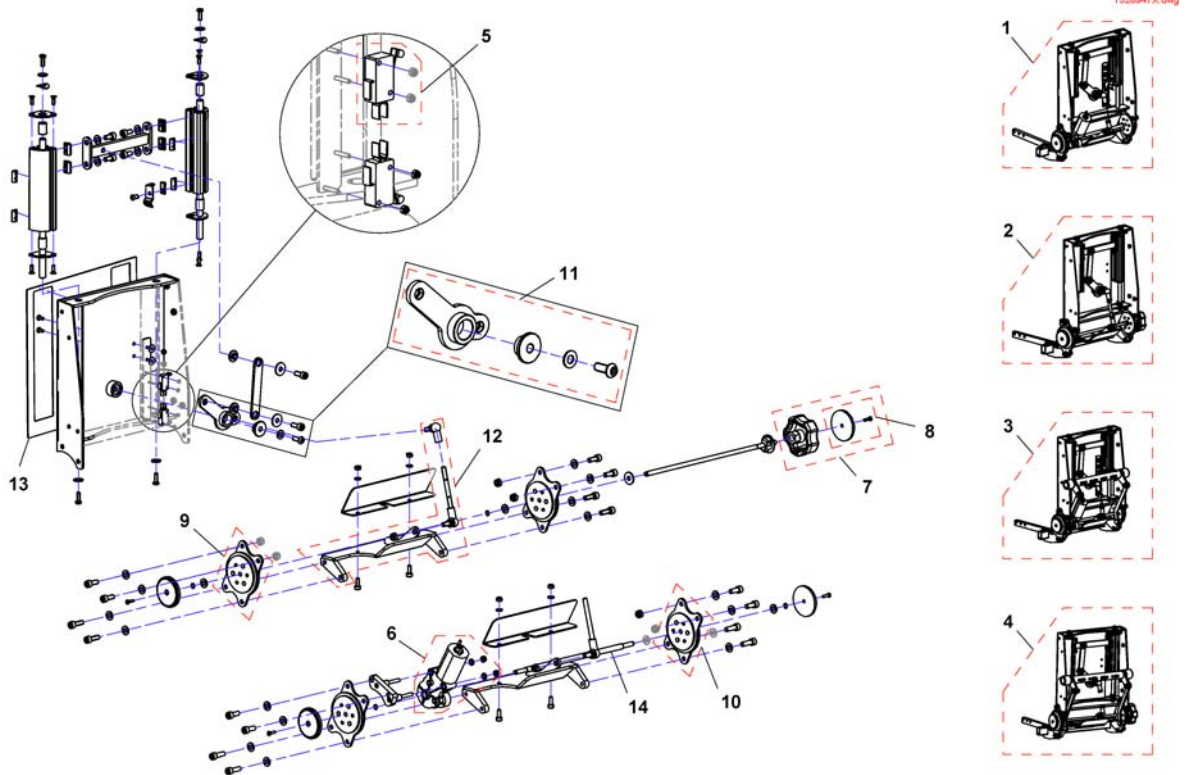

Pos.	Part number	Description	Valid from → until	Qty
-	SP1529390	Slowdown- / Tiltcable		1
-	SP1529529	Space-angle sensor with connector		1
1	SP1527126	Electrical Backrest, cpl.		1
2	SP1528947	Manuell Backrest, adjustment RH, cpl.		1
2	SP1528948	Manuell Backrest, adjustment LH, cpl.		1
3	SP1536037	Electrical Backrest, following Armrest, cpl.		1
4	SP1537377	Manuell Backrest, following armrest, adjustment RH, cpl.		1
4	SP1537378	Manuell Backrest, following armrest, adjustment LH, cpl.		1
5	1548506	No description available.		1
5	SP1526445	Cable loom for stop switch		1
6	SP1530520	Gearmotor		1
7	SP1420483	HAND WHEEL		1
8	SP1421696	PERFORATED REFLECTOR, YELLOW		1
9	SP1450234	SEAT BRACE MANUAL BACKREST, 1 PAIR		1
10	SP1423211	SEAT BRACE ELECTRIC BACKREST, 1 PAIR		1
11	SP1535255	Retrofitting-Kit Backrest		1
12	SP1537863	Bridge for shearcompensation		1
13	SP1532897	Backrest sliding plate		1
14	SP1516962	Motor shaft		1


Backrestcover for EasyAdapt

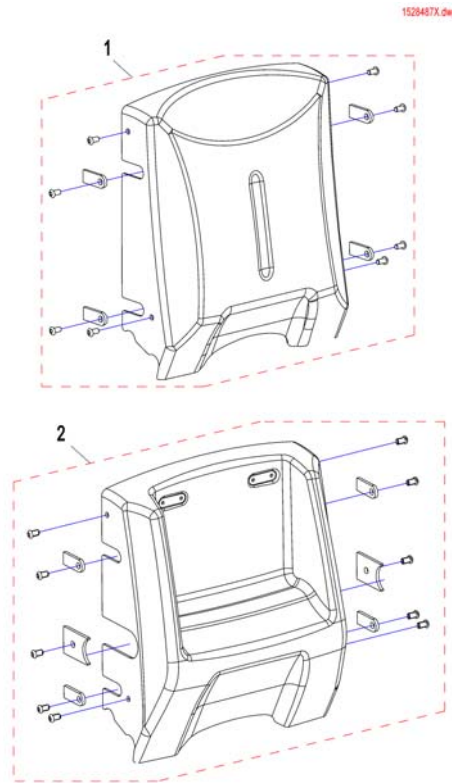

Pos.	Part number	Description	Valid from → until	Qty
1	SP1528487	Backrestcover for EasyAdpt		1
2	SP1537706	Backrestcover EasyAdapt for following armrest		1

Backplate EasyAdapt


1528949X.dwg

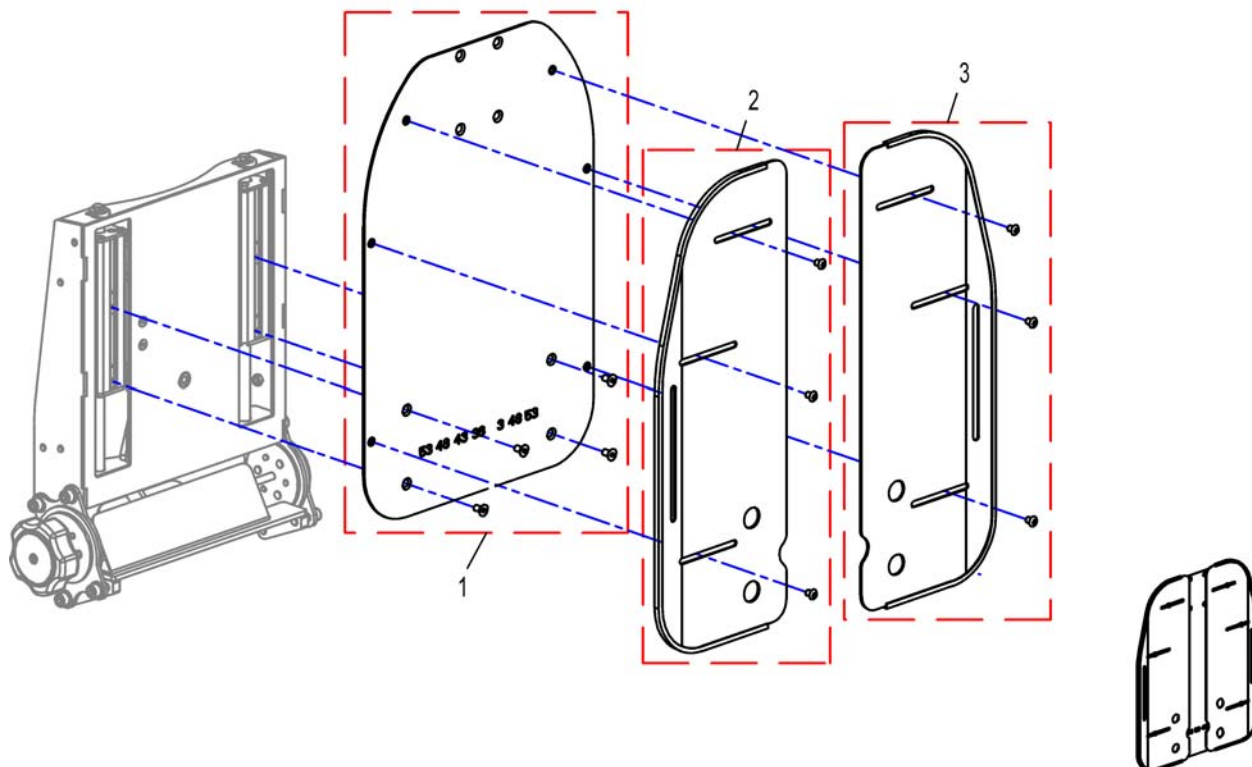

Pos.	Part number	Description	Valid from → until	Qty
1	SP1530525	Backplate EasyAdapt, middle		1
2	SP1530526	Backplate EasyAdapt, right		1
3	SP1530527	Backplate EasyAdapt, left		1

Backplate EasyAdapt for DAHL Docking Station

1579892 iso

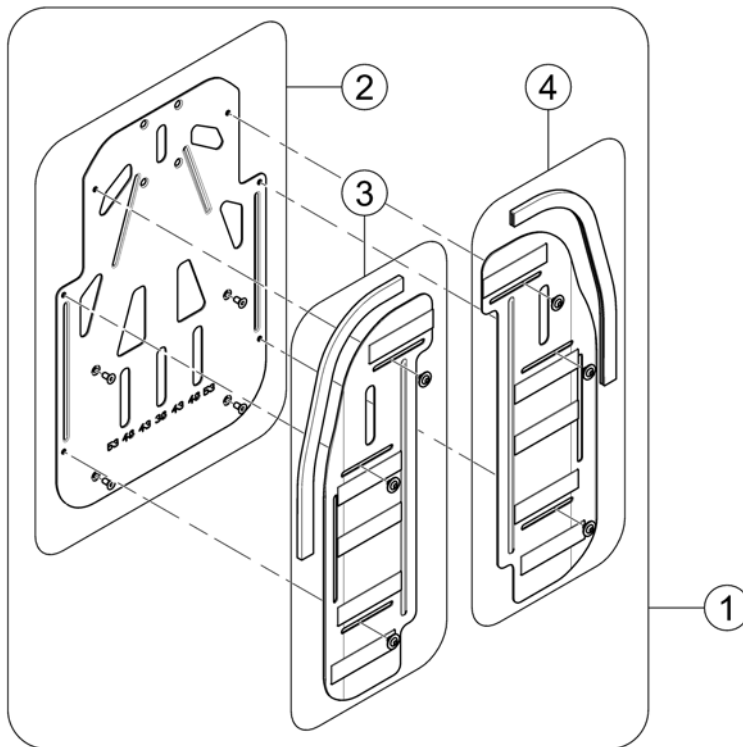

Pos.	Part number	Description	Valid from → until	Qty
1	SP1541384	Backrest Plate Assy, reinforced - for DAHL Docking Station		1
2	SP1579843	Center Backrest Plate, reinforced - for DAHL Docking Station		1
3	SP1579844	Back Plate left, reinforced - for DAHL Docking Station		1
4	SP1579845	Back Plate right, reinforced - for DAHL Docking Station		1


Backrest Cushions

Pos.	Part number	Description	Valid from → until	Qty
-		Backrest Cushions Laguna & Onyx		1
-		Matrx backrest		1

Backrest Cushions Laguna & Onyx

152905X-140

Laguna

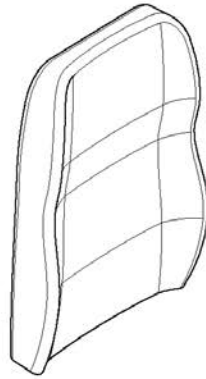

1	380mm		
2	430mm		
3	480mm		
4	530mm		
5	380mm		
6	430mm		
7	480mm		
8	530mm		
9	380mm		
10	430mm		
11	480mm		
12	530mm		
13	380mm		
14	430mm		
15	480mm		
16	530mm		

Onyx




17	400mm	500mm		
18	400mm	630mm		
19	430mm	500mm		
20	430mm	630mm		
21	450mm	500mm		
22	450mm	630mm		
23	480mm	500mm		
24	480mm	630mm		

Pos.	Part number	Description	Valid from → until	Qty
1	1444339	Backrest Cushion Laguna, 372x550mm, Grey Plush		1
2	1435798	Backrest Cushion Laguna, 402x550mm, Grey Plush		1
3	1444340	Backrest Cushion Laguna, 452x550mm, Grey Plush		1
4	1444341	Backrest Cushion Laguna, 492x550mm, Grey Plush		1
5	1427118	Backrest cushion comfort Laguna (XS), 372x550 mm, Dartex grey (TR23)	→ 2014-11-30	1
5	SP1529538-TR26	Backrest Cushion Laguna / EasyAdapt, Dartex black - SW 380mm	2014-12-01 →	1
6	-	No longer available - please choose Dartex black	→ 2014-11-30	1
6	SP1529539-TR26	Backrest Cushion Laguna / EasyAdapt, Dartex black - SW 430mm	2014-12-01 →	1
7	-	No longer available - please choose Dartex black	→ 2014-11-30	1
7	SP1529540-TR26	Backrest Cushion Laguna / EasyAdapt, Dartex black - SW 480mm	2014-12-01 →	1
8	1427121	Backrest Cushion Laguna, 492x550mm, Dartex Grey	→ 2014-11-30	1
8	SP1529545-TR26	Backrest Cushion Laguna / EasyAdapt, Dartex black - SW 530mm	2014-12-01 →	1
9	SP1529585	Backrest Cushion Laguna / EasyAdapt, black - SW 380mm		1
10	SP1529586	Backrest Cushion Laguna / EasyAdapt, black - SW 430mm		1
11	SP1529587	Backrest Cushion Laguna / EasyAdapt, black - SW 480mm		1
12	SP1529596	Backrest Cushion Laguna / EasyAdapt, black - SW 530mm		1
13	SP1529538	Backrest Cushion Laguna / EasyAdapt, black with orange seam - SW 380mm		1
14	SP1529539	Backrest Cushion Laguna / EasyAdapt, black with orange seam - SW 430mm		1
15	SP1529540	Backrest Cushion Laguna / EasyAdapt, black with orange seam - SW 480mm		1
16	SP1529545	Backrest Cushion Laguna / EasyAdapt, black with orange seam - SW 530mm		1
17	SP1532061	ONYX Back 40x50cm (16" x 20")		1
18	SP1532062	ONYX Back 40x63cm (16" x 25")		1
19	SP1532064	ONYX Back 43x50cm (17" x 20")		1
20	SP1532065	ONYX Back 43x63cm (17" x 25")		1
21	SP1532118	ONYX Back 45x50cm (18" x 20")		1
22	SP1532194	ONYX Back 45x63cm (18" x 25")		1
23	SP1532211	ONYX Back 48x50cm (19" x 20")		1
24	SP1532212	ONYX Back 48x63cm (19" x 25")		1


Matrix backrest

1637639 no

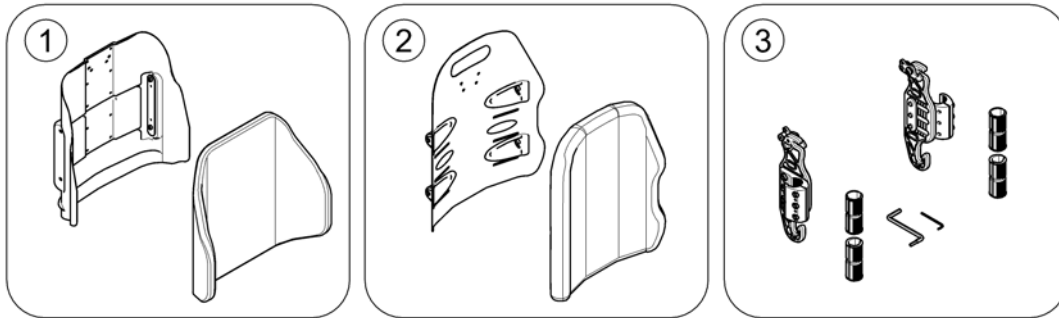

Pos.	Part number	Description	Valid from → until	Qty
1	-	Matrix Posture Back		1
2	-	Matrix Elite (Full) Back		1
3	-	Mounting Hardware for Matrix backrests		1

Posture Back

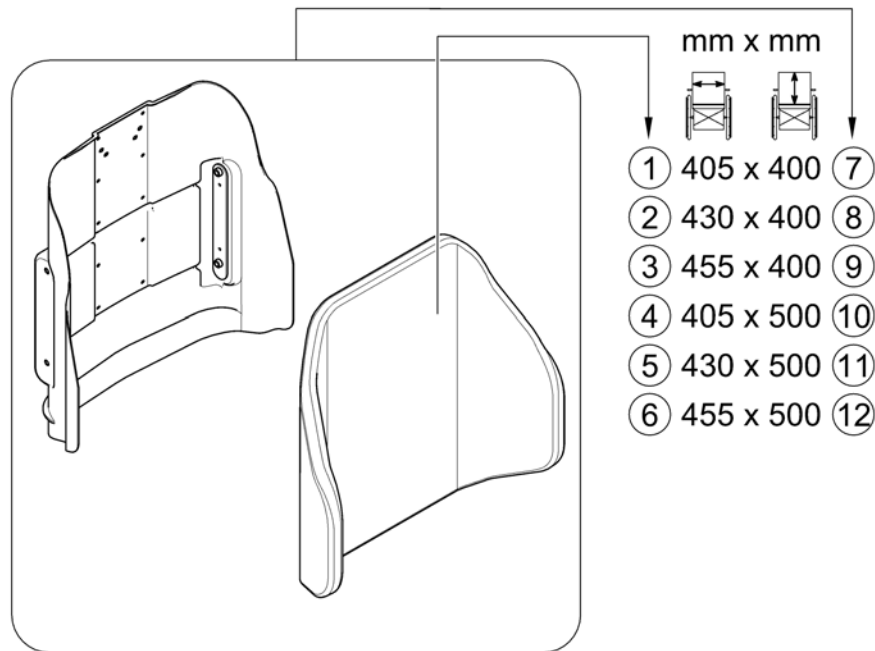


Pos.	Part number	Description	Valid from → until	Qty
1	1544896	Backrest Cushion - Posture Back 405x400mm (16x16")		1
2	1544899	Backrest Cushion - Posture Back 430x400mm (17x16")		1
3	1544995	Backrest Cushion - Posture Back 455x400mm (18x16")		1
4	1544897	Backrest Cushion - Posture Back 405x500mm (16x20")		1
5	1544982	Backrest Cushion - Posture Back 430x500mm (17x20")		1
6	1544996	Backrest Cushion - Posture Back 455x500mm (18x20")		1
7	SP1515727	Posture Back 405x400mm (16x16"), cpl. [2174]		1
8	SP1515729	Posture Back 430x400mm (17x16"), cpl. [2174]		1
9	SP1515731	Posture Back 455x400mm (18x16"), cpl. [2174]		1
10	SP1515728	Posture Back 405x500mm (16x20"), cpl. [2175]		1
11	SP1515730	Posture Back 430x500mm (17x20"), cpl. [2175]		1
12	SP1515732	Posture Back 455x500mm (18x20"), cpl. [2175]		1

Matrx Elite (Full) Back

158914-00


Elite Full Back

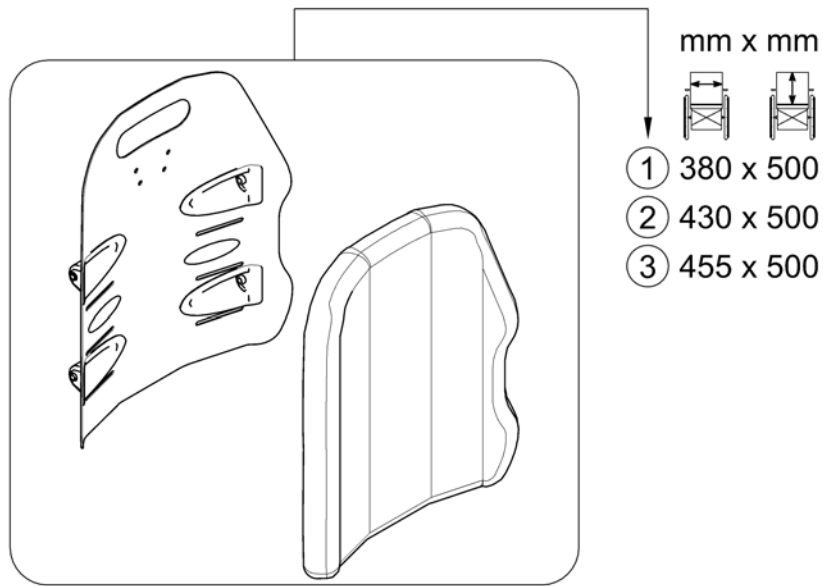

Pos.	Part number	Description	Valid from → until	Qty
1	SP1532094	Matrx Elite Back 15x20" [2176]		1
2	SP1517755	Matrx Elite Back 17x20" [2176]		1
3	SP1517756	Matrx Elite Back 18x20" [2176]		1

Mounting Hardware for Matrix backrests


163/807/00

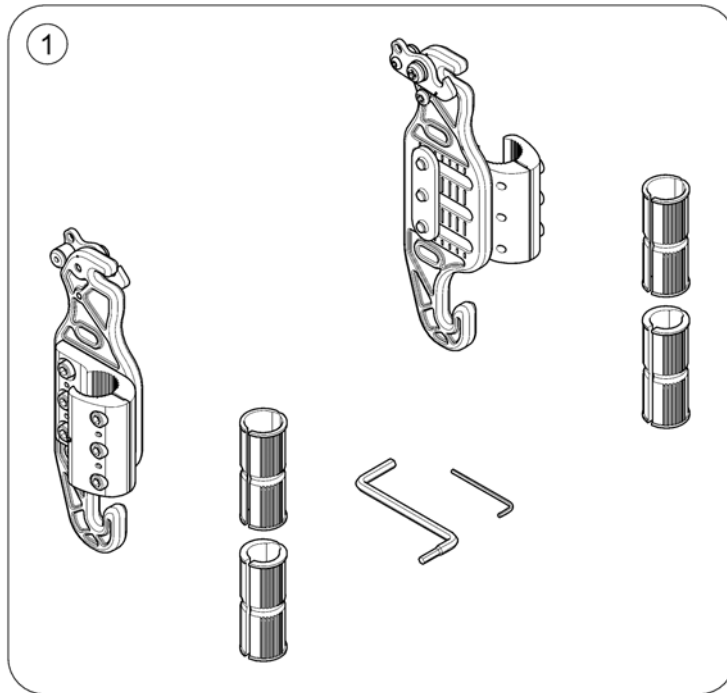

Pos.	Part number	Description	Valid from → until	Qty
1	SP1545275	Mounting hardware, Matrix Backrest		1

Push Handle EasyAdapt


1573324.dwg

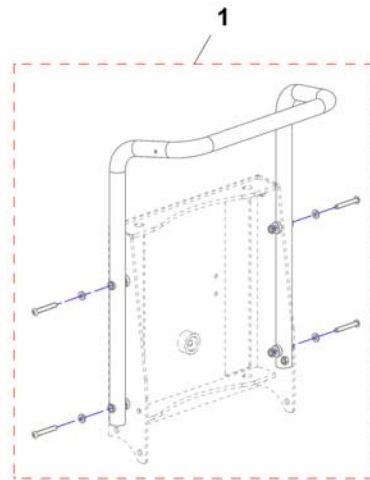

Pos.	Part number	Description	Valid from → until	Qty
1	SP1529001	Push Handle EasyAdapt-Seat		1

Tension adjustable Sling Back


1533409X.dwg

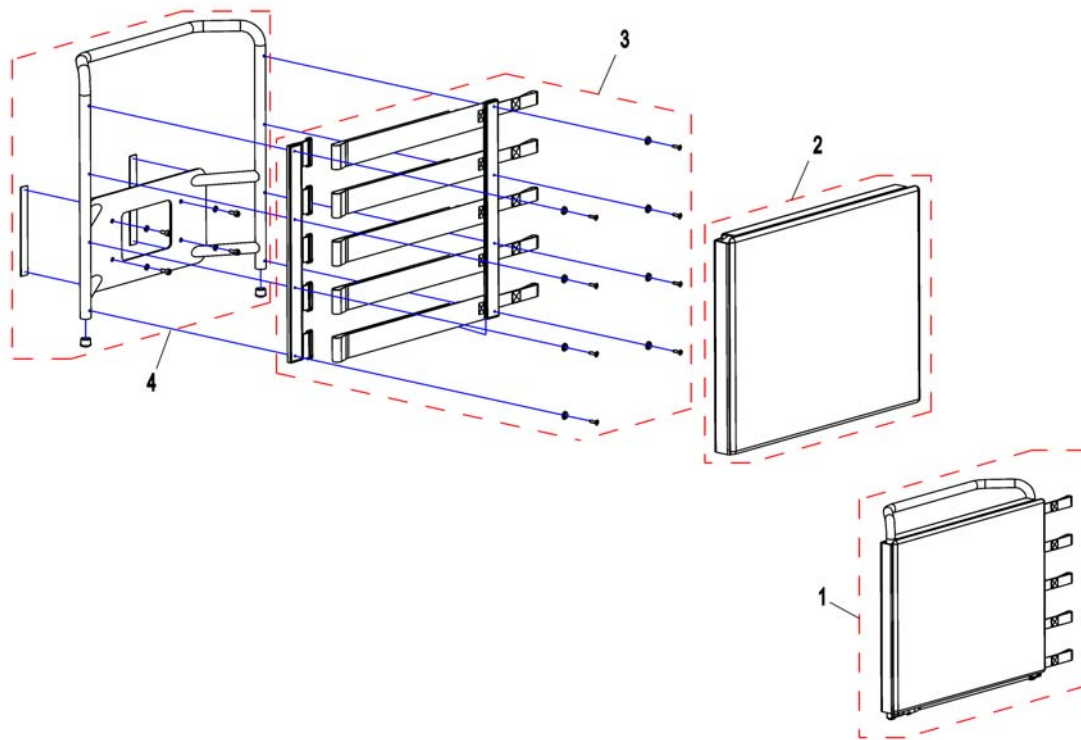

Pos.	Part number	Description	Valid from → until	Qty
1	SP1531825	Lashing Backrest SW 43, cpl.		1
1	SP1533409	Lashing Backrest SW 53, cpl.		1
2	SP1531830	Backrestupholstery SW 43		1
2	SP1533405	Backrestupholstery SW 53		1
3	SP1537258	Lashingstrap SW 43, cpl.		1
3	SP1537296	Lashingstrap SW 53, cpl.		1
4	SP1537297	Backrestcarrier SW 43, cpl.		1
4	SP1537298	Backrestcarrier SW 53, cpl.		1

"Universal"-Backrest frame


1531052X.dwg

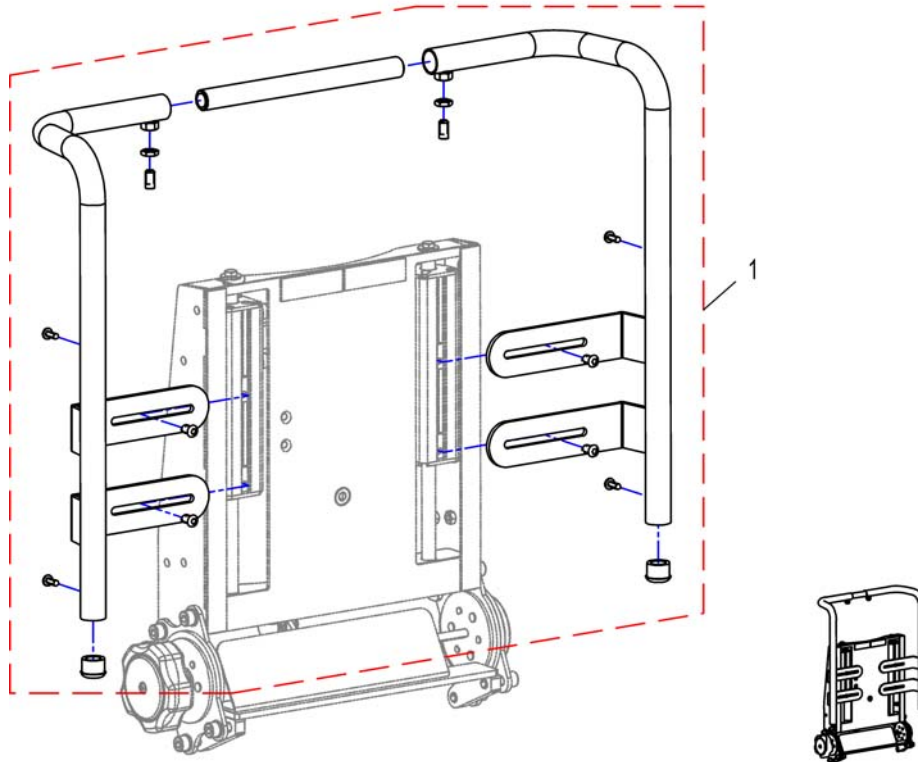

Pos.	Part number	Description	Valid from → until	Qty
1	SP1531052	"Universal"-Backrest frame		1

Backrest for Modulite Seat


1586095.kso

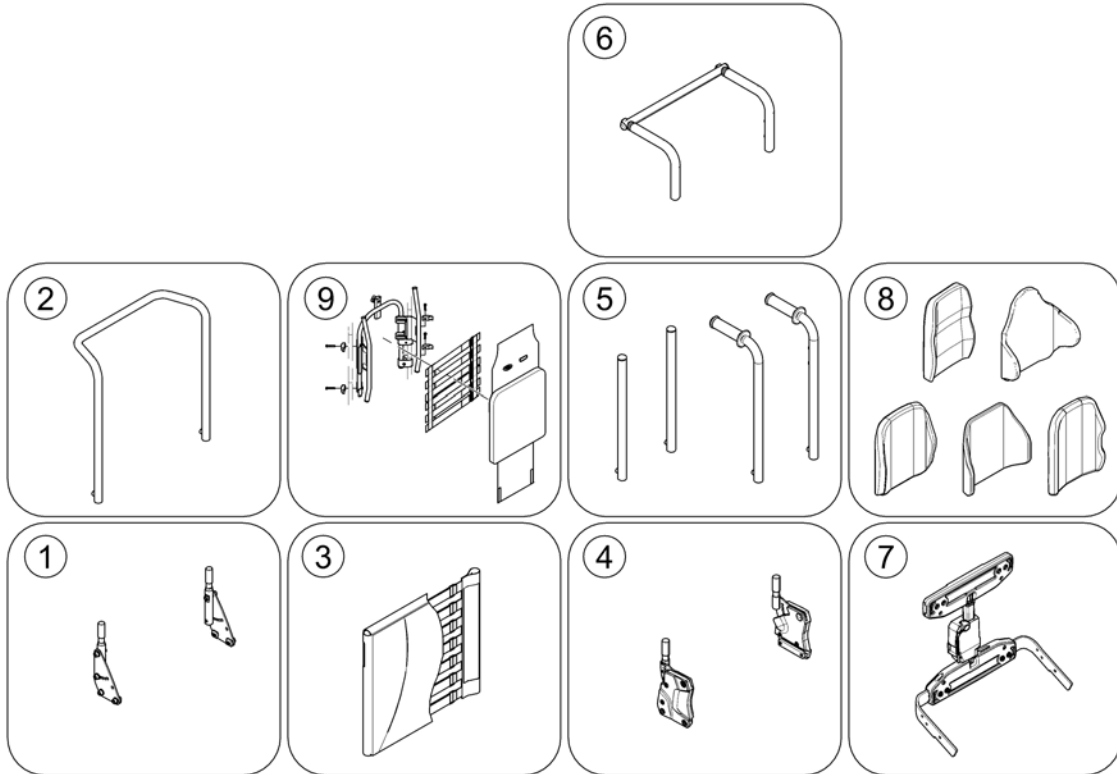

Pos.	Part number	Description	Valid from → until	Qty
1	-	Support for Backrest Frame [0975+2161]		1
2	-	Backrest Frame [2161]		1
3	-	Sling Back, tension adjustable [2171]		1
4	-	Backrest support		1
5	-	Backposts [2162/2163]		1
6	-	Connection tube [1307]		1
7	-	Back Recliner / Stabilization bracket [0977]		1
8	-	Backrest Cushions [2172/2173/2174/2175/2176/2177/2178]		1
9	-	Flex3 back, tension adjustable [2179]		1

Support for Backrest Frame


1571553 no

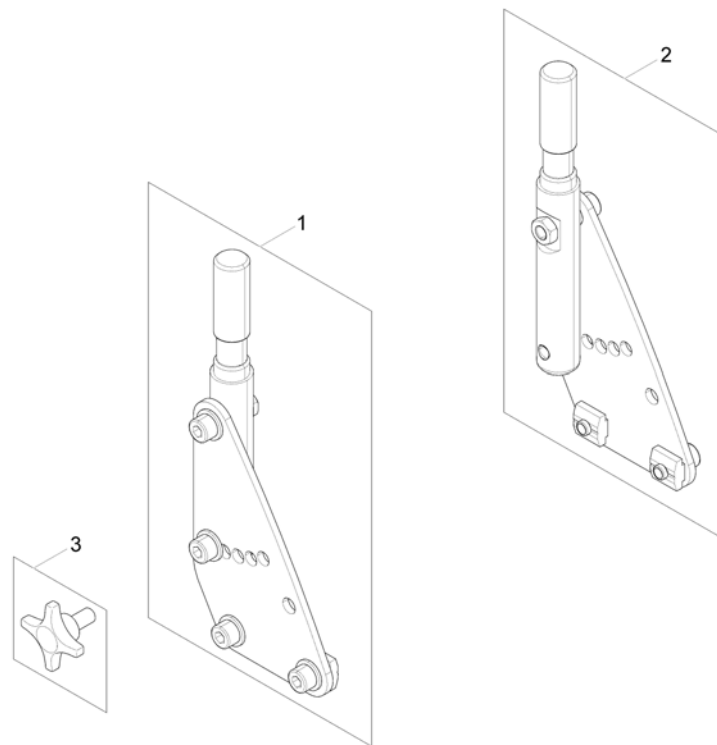

Pos.	Part number	Description	Valid from → until	Qty
1	SP1571542	Backrest Support RH		1
2	SP1571553	Backrest Support LH		1
3	5824117	hand wheel M8x15 <i>(only swedish homologation)</i>		2

Backrest Frame


1571831.00

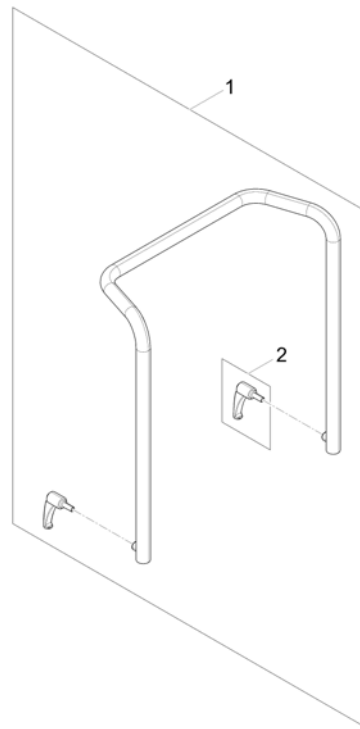


Pos.	Part number	Description	Valid from → until	Qty
1	SP1567810	Backrest Frame		1
2	0021-70002	Indexable lever 63mm M8x16		1

Sling Back, tension adjustable

1571829.000

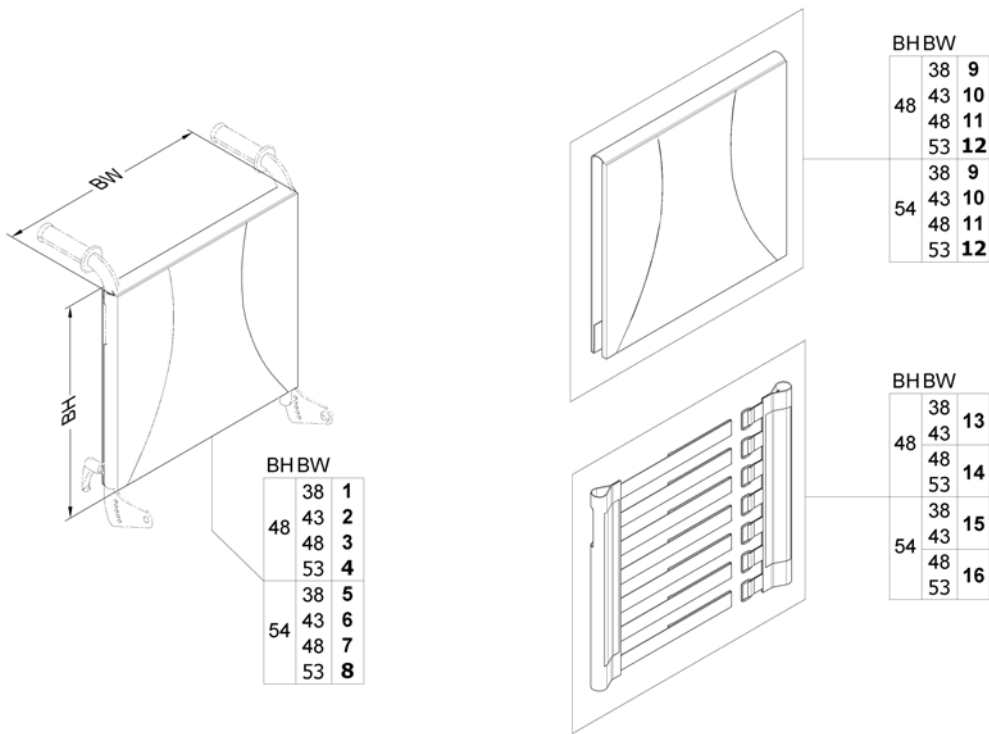


Pos.	Part number	Description	Valid from → until	Qty
1	SP1567529	Sling Back, tension adjustable & Upholstery, BW 380mm / BH 480mm, cpl.		1
2	SP1567465	Sling Back, tension adjustable & upholstery, BW 430mm / BH 480mm, cpl.		1
3	SP1567530	Sling Back, tension adjustable & upholstery, BW 480mm / BH 480mm, cpl.		1
4	SP1567531	Sling Back, tension adjustable & upholstery, BW 530mm / BH 480mm, cpl.		1
5	SP1567627	Sling Back, tension adjustable & upholstery, BW 380mm / BH 540mm, cpl.		1
6	SP1567746	Sling Back, tension adjustable & upholstery, BW 430mm / BH 540mm, cpl.		1
7	SP1567747	Sling Back, tension adjustable & upholstery, BW 480mm / BH 540mm, cpl.		1
8	SP1567748	Sling Back, tension adjustable & upholstery, BW 530mm / BH 540mm, cpl.		1
9	SP1563563	Backrest upholstery		1
10	SP1563653	Backrest upholstery		1
11	SP1563654	Backrest upholstery		1
12	SP1563655	Backrest upholstery		1
13	SP1571492	Sling back		1
14	SP1571493	Sling back		1
15	SP1571494	Sling back		1
16	SP1571510	Sling back		1

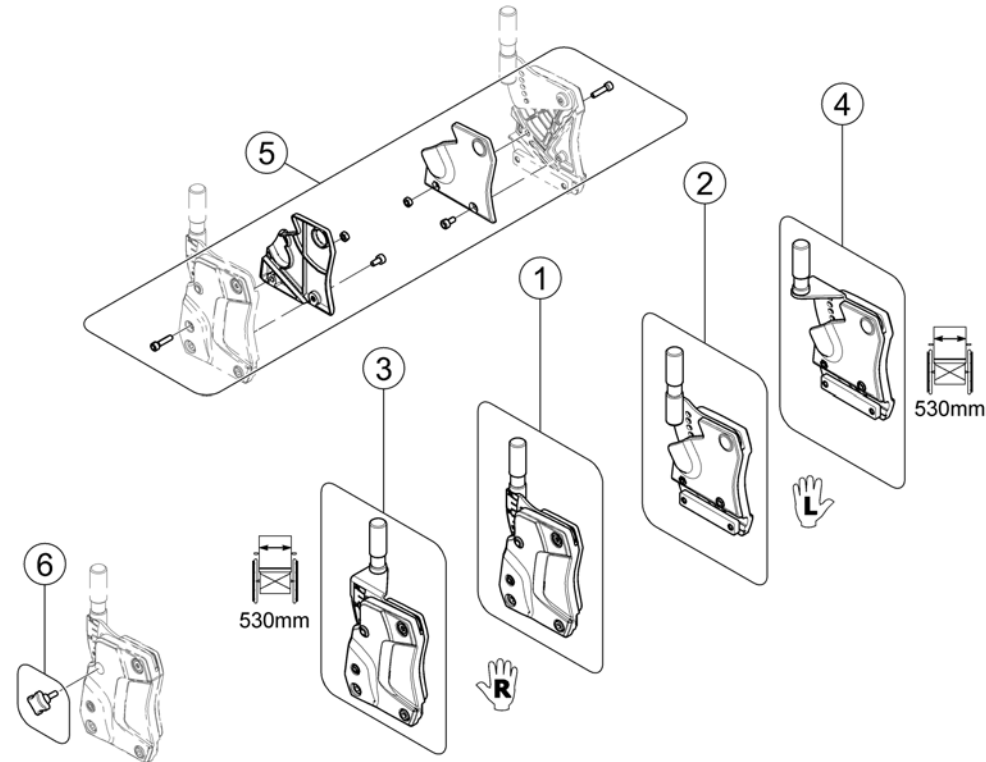

Pos.	Part number	Description	Valid from → until	Qty
-	-	Design change on 02. May 2016 - we suggest to change older supports on both sides.		1
1	SP1582709	Backrest Support, cpl. - RH <i>Compatible with Flip-Up & Following Armrest [2193/2192/2194] - replaces SP1571560</i>		1
2	SP1582708	Backrest Support, cpl. - LH <i>Compatible with Flip-Up & Following Armrest [2193/2192/2194] - replaces SP1571561</i>		1
3	SP1582952	Backrest Support, cpl. - RH (SW 530mm) <i>Compatible with Flip-Up & Following Armrest [2193/2192/2194] - replaces SP1571568</i>		1
4	SP1582951	Backrest Support, cpl. - LH (SW 530mm) <i>Compatible with Flip-Up & Following Armrest [2193/2192/2194] - replaces SP1571569</i>		1
5	SP1571676	Covers		1
6	SP1579877	Cross Handle M6 <i>(Swedish Homologation only!)</i>	→ 2015-07-31	2
6	SP1589324	Cross handle M8 <i>(Swedish Homologation only!)</i>	2015-08-01 →	2
7	SP1590656	Hole plug (for powered backrest only)	2016-07-01 →	1


Backposts

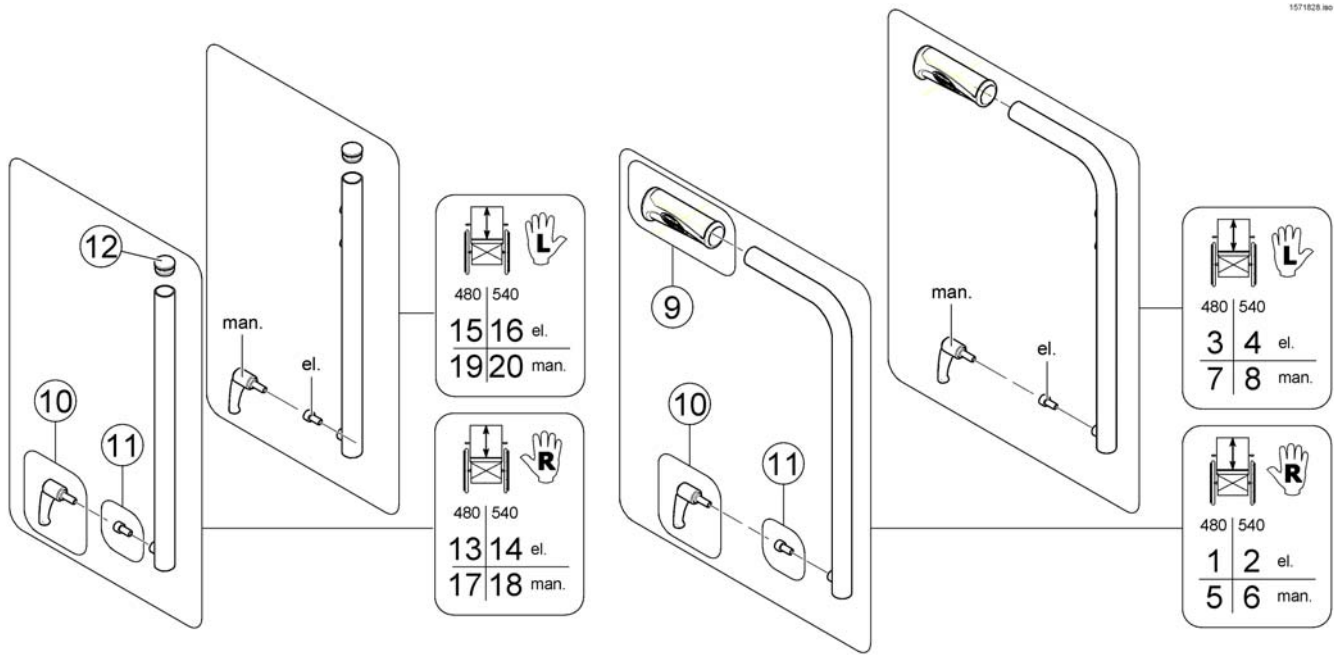

Pos.	Part number	Description	Valid from → until	Qty
1	SP1587177	Backpost el. BH 480mm - RH <i>Compatible with Flip-Up & Following Armrest [2193/2192/2194] - replaces SP1563102</i>		1
2	SP1587184	Backpost el. BH 540mm - RH <i>Compatible with Flip-Up & Following Armrest [2193/2192/2194] - replaces SP1563103</i>		1
3	SP1587176	Backpost el. BH 480mm - LH <i>Compatible with Flip-Up & Following Armrest [2193/2192/2194] - replaces SP1560576</i>		1
4	SP1587183	Backpost el. BH 540mm - LH <i>Compatible with Flip-Up & Following Armrest [2193/2192/2194] - replaces SP1559578</i>		1
5	SP1587163	Backpost man. BH 480mm - RH <i>Compatible with Flip-Up & Following Armrest [2193/2192/2194] - replaces SP1572489</i>		1
6	SP1587182	Backpost man. BH 540mm - RH <i>Compatible with Flip-Up & Following Armrest [2193/2192/2194] - replaces SP1572490</i>		1
7	SP1587162	Backpost man. BH 480mm - LH <i>Compatible with Flip-Up & Following Armrest [2193/2192/2194] - replaces SP1572491</i>		1
8	SP1587178	Backpost man. BH 540mm - LH <i>Compatible with Flip-Up & Following Armrest [2193/2192/2194] - replaces SP1572492</i>		1
9	SP1586122	Handles (pair) <i>replaces item SP1563651</i>		1
10	0021-70002	Indexable lever 63mm M8x16		1
11	1567999	Bolt M8x16 8.8 black DIN912		1
13	SP1606879	Backpost el. BH 480mm - RH w/o Pushhandle		1
14	SP1606893	Backpost el. BH 540mm - RH w/o Pushhandle		1
15	SP1606880	Backpost el. BH 480mm - LH w/o Pushhandle		1
16	SP1606894	Backpost el. BH 540mm - LH w/o Pushhandle		1
17	SP1606877	Backpost man. BH 480mm - RH w/o Pushhandle		1
18	SP1606881	Backpost man. BH 540mm - RH w/o Pushhandle		1
19	SP1606878	Backpost man. BH 480mm - LH w/o Pushhandle		1
20	SP1606882	Backpost man. BH 540mm - LH w/o Pushhandle		1

Connection tube


1571835 iso

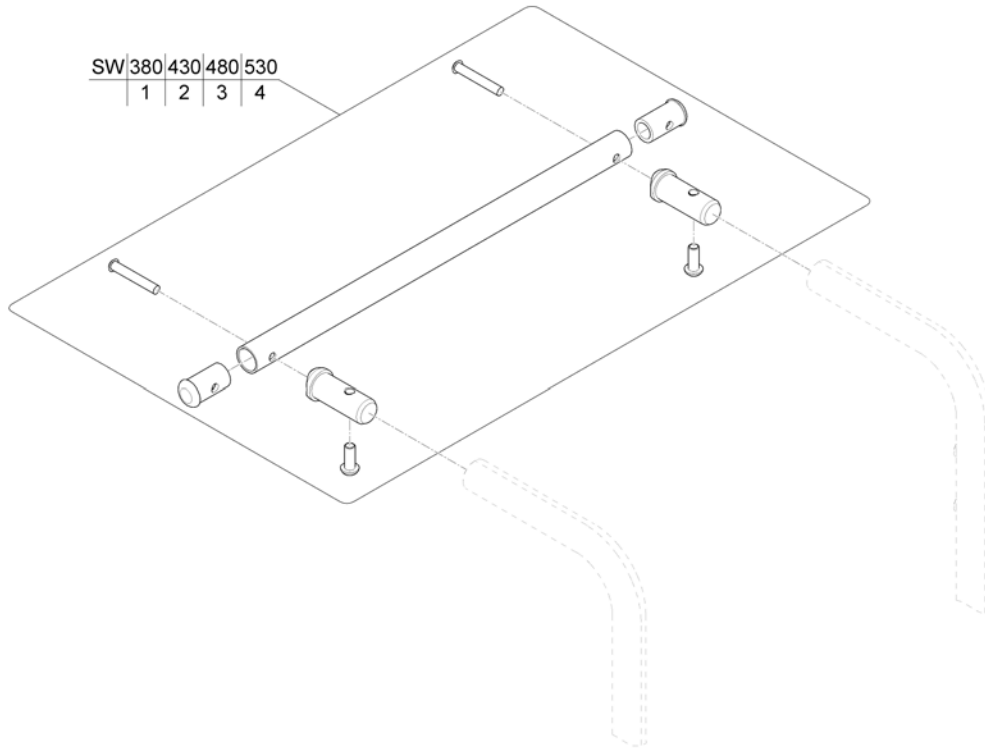


Pos.	Part number	Description	Valid from → until	Qty
1	SP1570370	Connection tube		1
2	SP1570371	Connection tube		1
3	SP1570372	Connection tube		1
4	SP1570373	Connection tube		1

Back Recliner / Stabilization bracket

1571097.kso

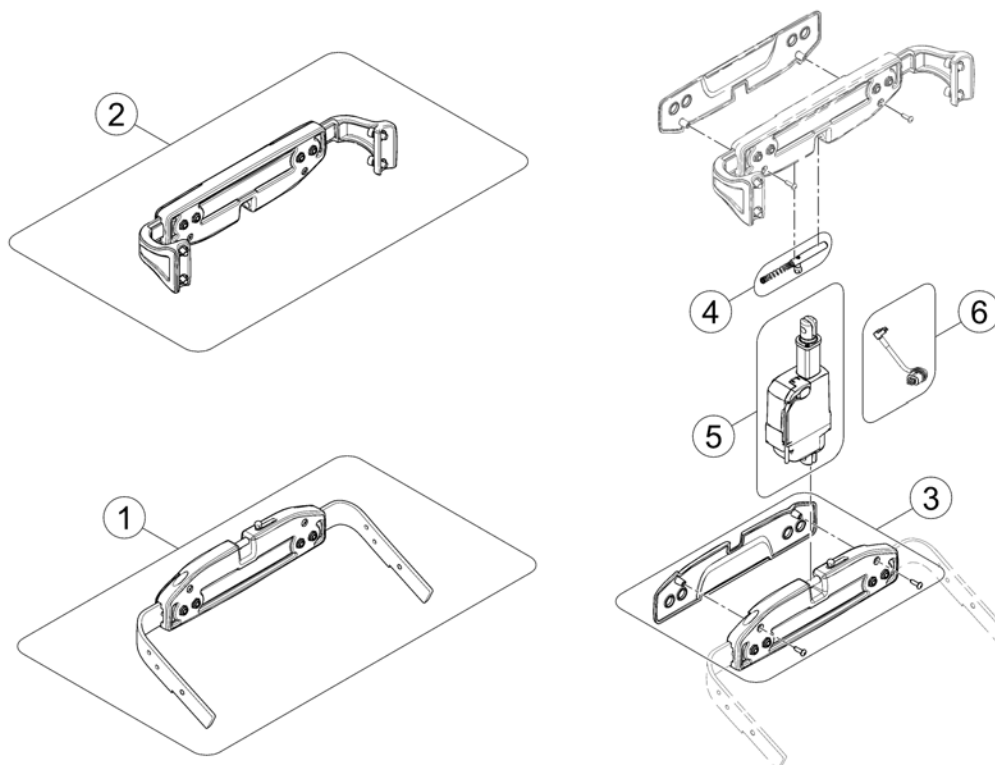


Pos.	Part number	Description	Valid from → until	Qty
1	SP1567169	Actuator bracket (pre set to SW 43cm)		1
2	SP1567168	Stabilization bracket (pre set to SW 43cm)		1
3	SP1571702	Cover		1
4	SP1571704	Locking bolt		1
5	SP1557820	Actuator LA23		1
6	SP1572166	Actuator Cable LA23 ACS2 (~950mm)		1

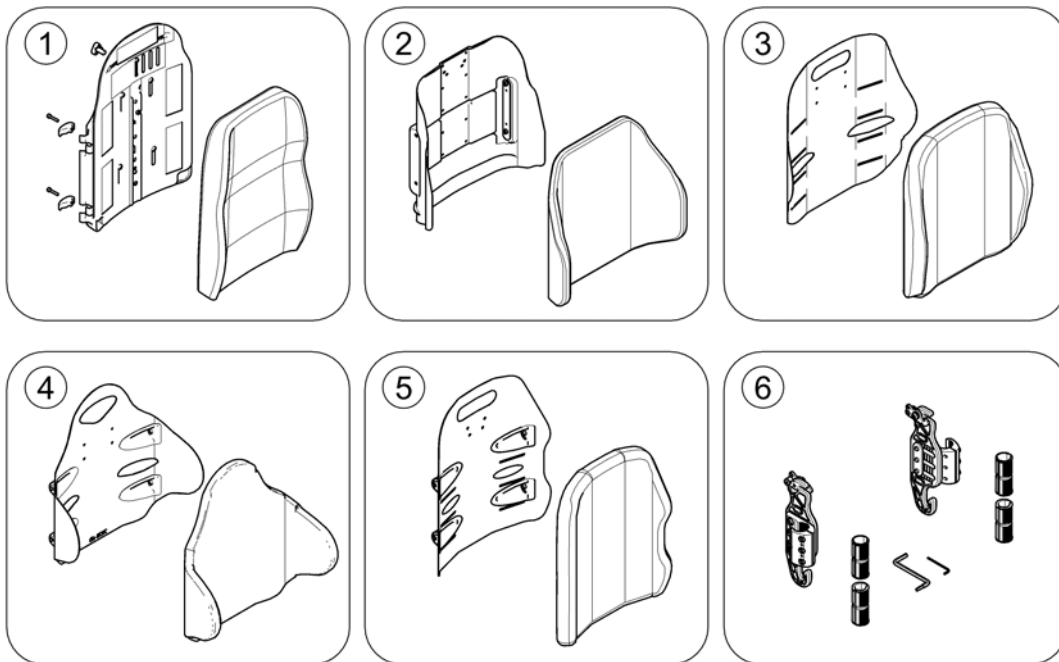

Pos.	Part number	Description	Valid from → until	Qty
1	-	Flex 3 / Laguna		1
2	-	Matrx Posture Back		1
3	-	Matrx Elite T/R Back	2015-07-01 →	1
4	-	Matrx Elite Deep Back	2015-07-01 →	1
5	-	Matrx Elite (Full) Back		1
6	-	Mounting Hardware for Matrx backrests		1


Flex 3 / Laguna

1573106.mw

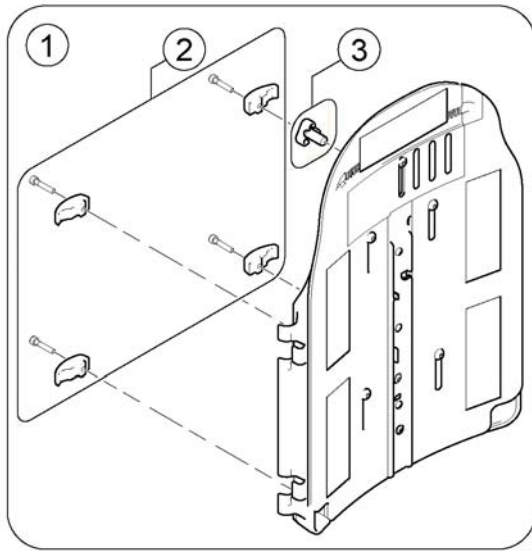

Flex 3

Laguna

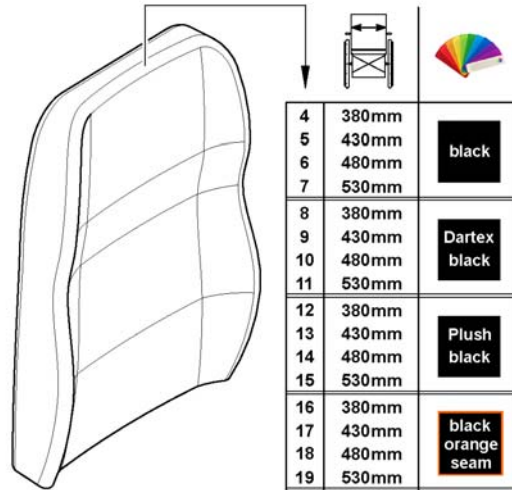

Pos.	Part number	Description	Valid from → until	Qty
1	SP1568489	Flex3 backrest plate <i>[2172]</i>		1
2	SP1572321	Clamps		1
3	1497411	HANDWHEEL M12 LONG NECK/HEADREST		1
4	SP1529585	Backrest Cushion Laguna / EasyAdapt, black - SW 380mm <i>[2173+2151]</i>		1
5	SP1529586	Backrest Cushion Laguna / EasyAdapt, black - SW 430mm <i>[2173+2151]</i>		1
6	SP1529587	Backrest Cushion Laguna / EasyAdapt, black - SW 480mm <i>[2173+2151]</i>		1
7	SP1529596	Backrest Cushion Laguna / EasyAdapt, black - SW 530mm <i>[2173+2151]</i>		1
8	SP1529538-TR26	Backrest Cushion Laguna / EasyAdapt, Dartex black - SW 380mm <i>[2173+2152]</i>		1
9	SP1529539-TR26	Backrest Cushion Laguna / EasyAdapt, Dartex black - SW 430mm <i>[2173+2152]</i>		1
10	SP1529540-TR26	Backrest Cushion Laguna / EasyAdapt, Dartex black - SW 480mm <i>[2173+2152]</i>		1
11	SP1529545-TR26	Backrest Cushion Laguna / EasyAdapt, Dartex black - SW 530mm <i>[2173+2152]</i>		1
12	SP1529538-TR35	Backrest Cushion Laguna, Plush black - SW 380mm <i>[2173+2154]</i>		1
13	SP1529539-TR35	Backrest Cushion Laguna, Plush black - SW 430mm <i>[2173+2154]</i>		1
14	SP1529540-TR35	Backrest Cushion Laguna, Plush black - SW 480mm <i>[2173+2154]</i>		1
15	SP1529545-TR35	Backrest Cushion Laguna, Plush black - SW 530mm <i>[2173+2154]</i>		1
16	SP1529538	Backrest Cushion Laguna / EasyAdapt, black with orange seam - SW 380mm <i>[2173+2153]</i>		1
17	SP1529539	Backrest Cushion Laguna / EasyAdapt, black with orange seam - SW 430mm <i>[2173+2153]</i>		1
18	SP1529540	Backrest Cushion Laguna / EasyAdapt, black with orange seam - SW 480mm <i>[2173+2153]</i>		1


Flex 3 / Laguna

Pos.	Part number	Description	Valid from → until	Qty
19	SP1529545	Backrest Cushion Laguna / EasyAdapt, black with orange seam - SW 530mm <i>[2173+2153]</i>		1

Matrx Elite T/R Back

158006 Rev


Elite T/R Back

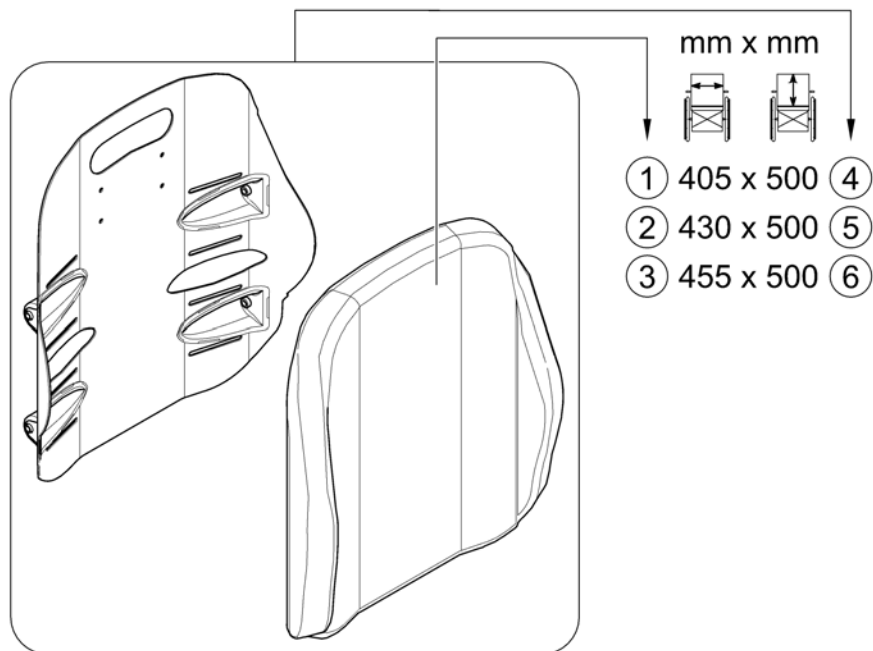

Pos.	Part number	Description	Valid from → until	Qty
1	1580110	Backrest Cushion - Elite T/R Back 405x500mm (16x20")		1
2	1580297	Backrest Cushion - Elite T/R Back 430x500mm (17x20")		1
3	1580298	Backrest Cushion - Elite T/R Back 455x500mm (18x20")		1
4	SP1580029	Elite T/R Back 405x500mm (16x20"), cpl. [2177+2121]		1
5	SP1580030	Elite T/R Back 430x500mm (17x20"), cpl. [2177+2122]		1
6	SP1580031	Elite T/R Back 455x500mm (18x20"), cpl. [2177+2123]		1

158913.00


Elite Deep Back

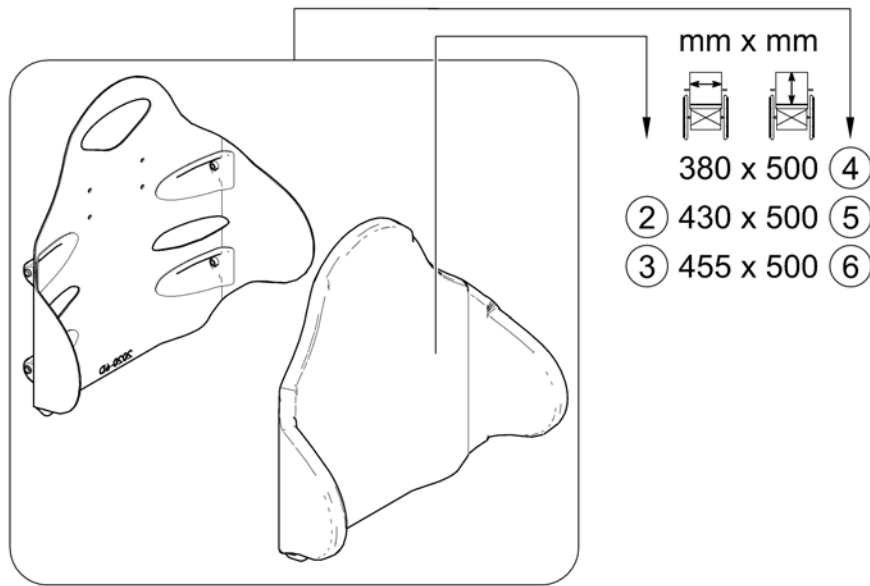

Pos.	Part number	Description	Valid from → until	Qty
2	1544077	Backrest Cushion - Elite Deep Back 430x500mm (17x20")		1
3	1544081	Backrest Cushion - Elite Deep Back 455x500mm (18x20")		1
4	SP1543511	Elite Deep Back 380x500mm (15x20"), cpl. [2178+2121]		1
5	SP1543519	Elite Deep Back 430x500mm (17x20"), cpl. [2178+2122]		1
6	SP1543523	Elite Deep Back 455x500mm (18x20"), cpl. [2178+2123]		1

Flex3 back, tension adjustable


1602719.iss

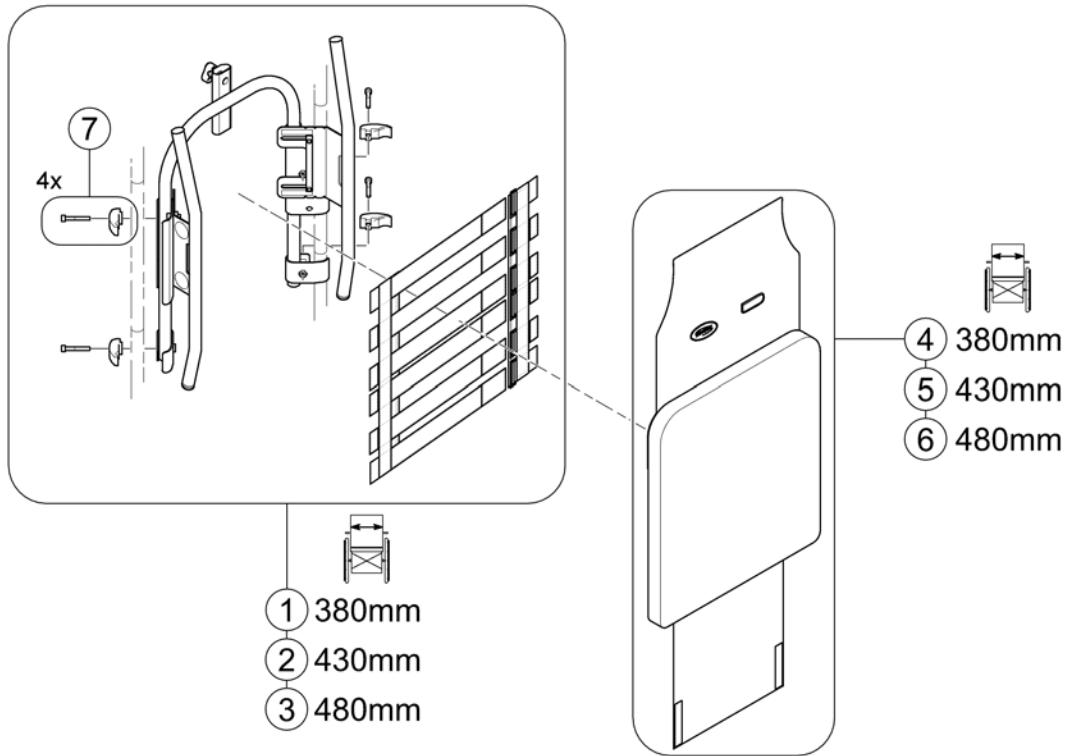

Pos.	Part number	Description	Valid from → until	Qty
1	SP1579675	Flex 3 Backrest, tension adjustable, cpl. - 390mm		1
2	SP1579692	Flex 3 Backrest, tension adjustable, cpl. - 440mm		1
3	SP1579693	Flex 3 Backrest, tension adjustable, cpl. - 490mm		1
4	SP1543667	Backrest cushion Lateral 475x540mm, Dartex black (TR26)		1
5	SP1543669	Backrest cushion Lateral 525x540mm, Dartex black (TR26)		1
6	SP1543671	Backrest cushion Lateral 575x540mm, Dartex black (TR26)		1
7	SP1580557	Mounting Hardware (Kit of 4)		1

Invacare® Storm4 | Backrest
Backrest for Ultra Low Maxx Seat


156815.000

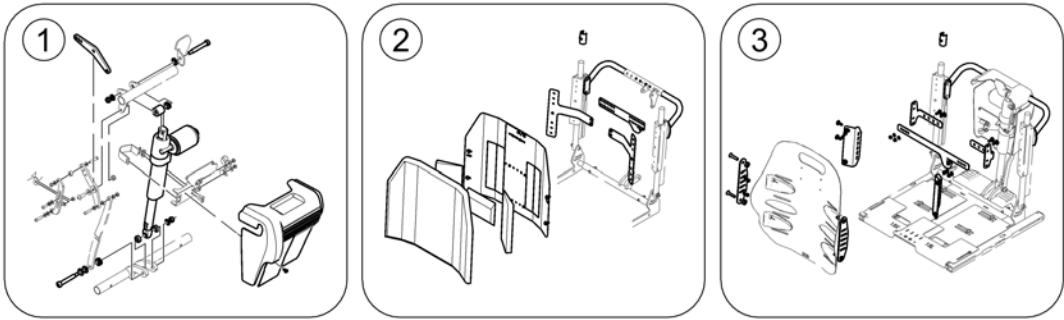

Pos.	Part number	Description	Valid from → until	Qty
-		Backrest Recliner		1
-		High Back		1
-		Matrix Back		1

Backrest Recliner

1588154 Rev


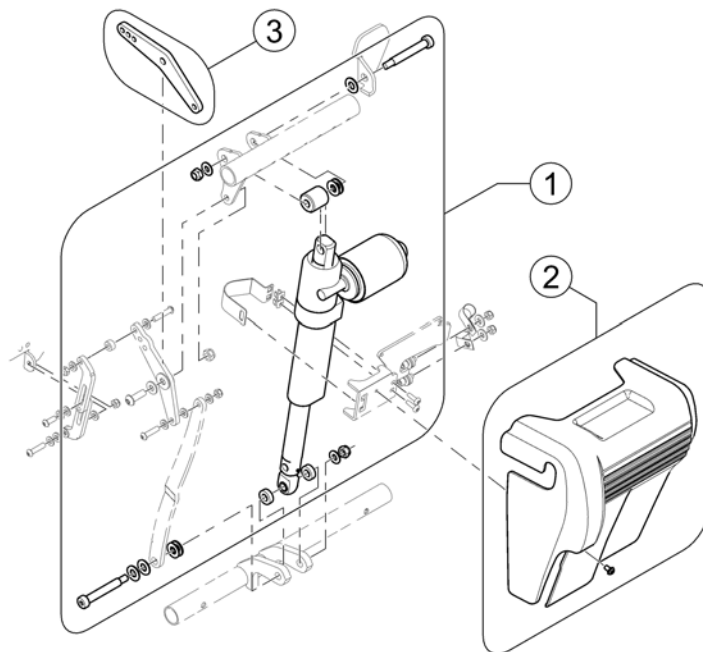


Pos.	Part number	Description	Valid from → until	Qty
1	SPTRC2191	Backrest Recliner with Hardware		1
	SPTRC2138	Backrest shroud with mounting hardware		1
3	SPTRC1926	Backrest Preline Crankarm 30°		1
-	SPTRE0322	Actuator Cable, Backrest Recliner		1

High Back

1588155.kim


High Back

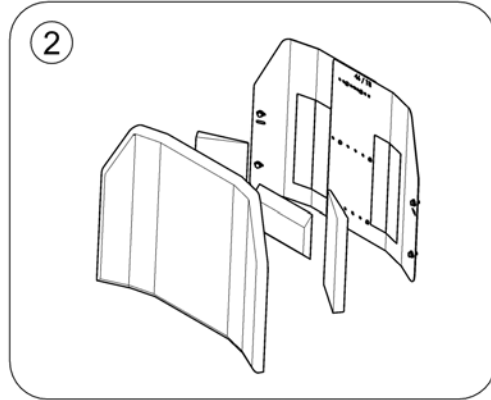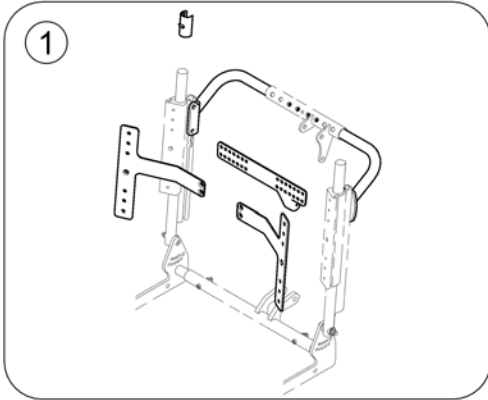


Pos.	Part number	Description	Valid from → until	Qty
1	-	Backrest Hardware		1
2	-	Backrest Cushions		1

Backrest Hardware

158200 Rev

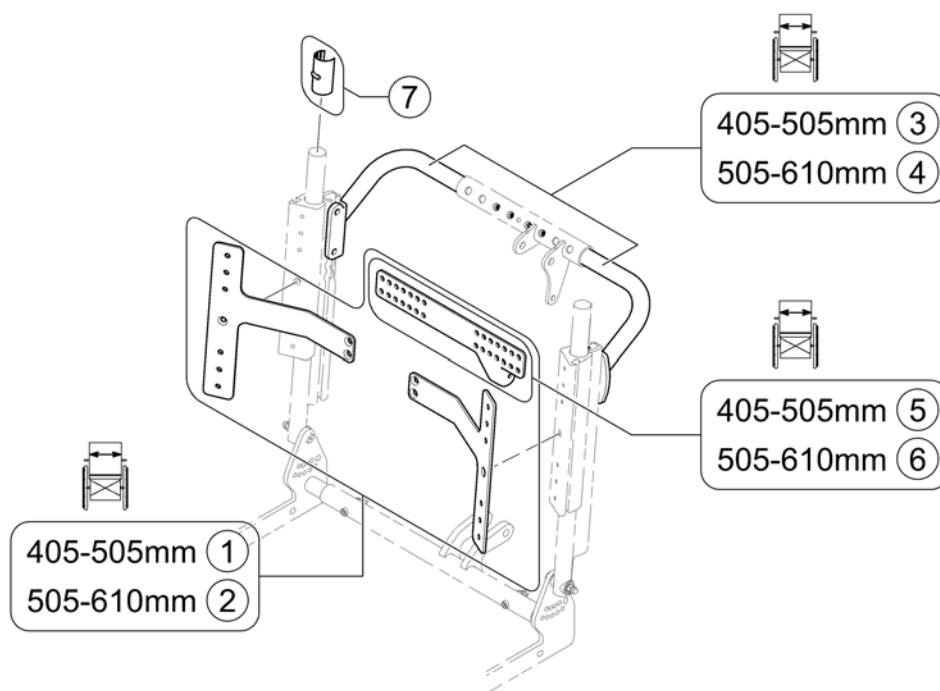

Pos.	Part number	Description	Valid from → until	Qty
1	SPTRC2136	High Back Interface Kit SW 405-505mm (16-20")		1
2	SPTRC2137	High Back Interface Kit SW 505-610mm (19-24")		1
3	SPTRC2188	Push Bars SW 405-505mm (16-20")		1
4	SPTRC2190	Push Bars SW 505-610mm (19-24")		1
5	SPTRX8497	High Back Center Plate SW 405-505mm (16-20")		1
6	SPTRX8655	High Back Center Plate SW 505-610mm (19-24")		1
7	SPTRX7883	Linear Bearing		4

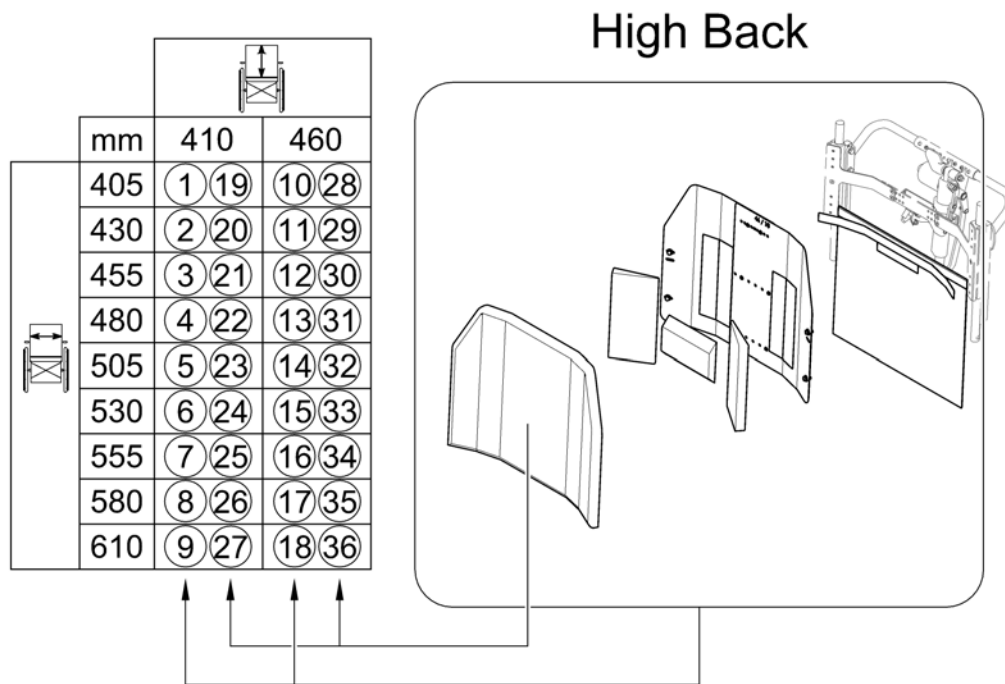

Pos.	Part number	Description	Valid from → until	Qty
1	SP1583769	High Back 405x410mm (16x16"), cpl.		1
2	SP1583772	High Back 430x410mm (17x16"), cpl.		1
3	SP1583774	High Back 405x410mm (18x16"), cpl.		1
4	SP1583857	High Back 480x410mm (19x16"), cpl.		1
5	SP1583930	High Back 505x410mm (20x16"), cpl.		1
6	SP1583932	High Back 530x410mm (21x16"), cpl.		1
7	SP1584053	High Back 555x410mm (22x16"), cpl.		1
8	SP1584055	High Back 580x410mm (23x16"), cpl.		1
9	SP1584081	High Back 610x410mm (24x16"), cpl.		1
10	SP1583771	High Back 405x460mm (16x18"), cpl.		1
11	SP1583773	High Back 430x460mm (17x18"), cpl.		1
12	SP1583775	High Back 455x460mm (18x18"), cpl.		1
13	SP1583858	High Back 480x460mm (19x18"), cpl.		1
14	SP1583931	High Back 505x460mm (20x18"), cpl.		1
15	SP1583934	High Back 530x460mm (21x18"), cpl.		1
16	SP1584054	High Back 555x460mm (22x18"), cpl.		1
17	SP1584056	High Back 580x460mm (23x18"), cpl.		1
18	SP1584118	High Back 610x460mm (24x18"), cpl.		1
19	SPTRS0425-1616	High Back Cover and foam 405x410mm (16x16")		1
20	SPTRS0425-1716	High Back Cover and foam 430x410mm (17x16")		1
21	SPTRS0425-1816	High Back Cover and foam 455x410mm (18x16")		1
22	SPTRS0425-1916	High Back Cover and foam 480x410mm (19x16")		1
23	SPTRS0425-2016	High Back Cover and foam 505x410mm (20x16")		1
24	SPTRS0425-2116	High Back Cover and foam 530x410mm (21x16")		1
25	SPTRS0425-2216	High Back Cover and foam 555x410mm (22x16")		1
26	SPTRS0425-2316	High Back Cover and foam 580x410mm (23x16")		1
27	SPTRS0425-2416	High Back Cover and foam 610x410mm (24x16")		1
28	SPTRS0425-1618	High Back Cover and foam 405x460mm (16x18")		1
29	SPTRS0425-1718	High Back Cover and foam 430x460mm (17x18")		1
30	SPTRS0425-1818	High Back Cover and foam 455x460mm (18x18")		1
31	SPTRS0425-1918	High Back Cover and foam 480x460mm (19x18")		1


Backrest Cushions

Pos.	Part number	Description	Valid from → until	Qty
32	SPTRS0425-2018	High Back Cover and foam 505x460mm (20x18")		1
33	SPTRS0425-2118	High Back Cover and foam 430x460mm (21x18")		1
34	SPTRS0425-2218	High Back Cover and foam 455x460mm (22x18")		1
35	SPTRS0425-2318	High Back Cover and foam 480x460mm (23x18")		1
36	SPTRS0425-2418	High Back Cover and foam 610x460mm (24x18")		1

Matrx Back

1588242 Rev


Matrx Back

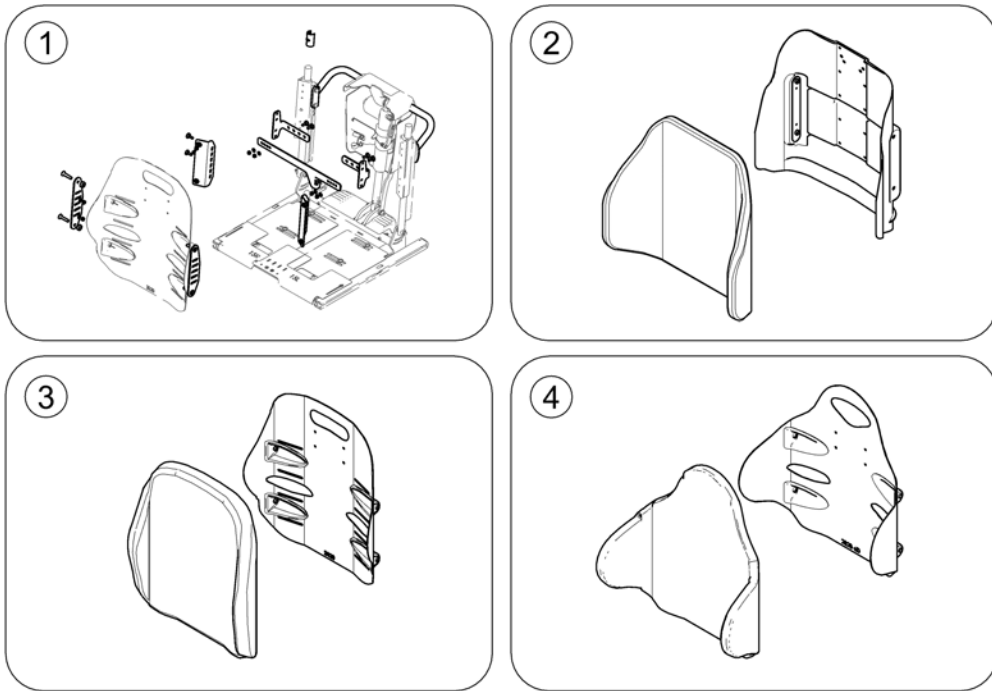

Pos.	Part number	Description	Valid from → until	Qty
1	-	Backrest Hardware		1
2	-	Posture Back		1
3	-	Elite T/R Back		1
4	-	Elite Deep Back		1
-	-	Privacy Flaps		1

Backrest Hardware


158203 Rev

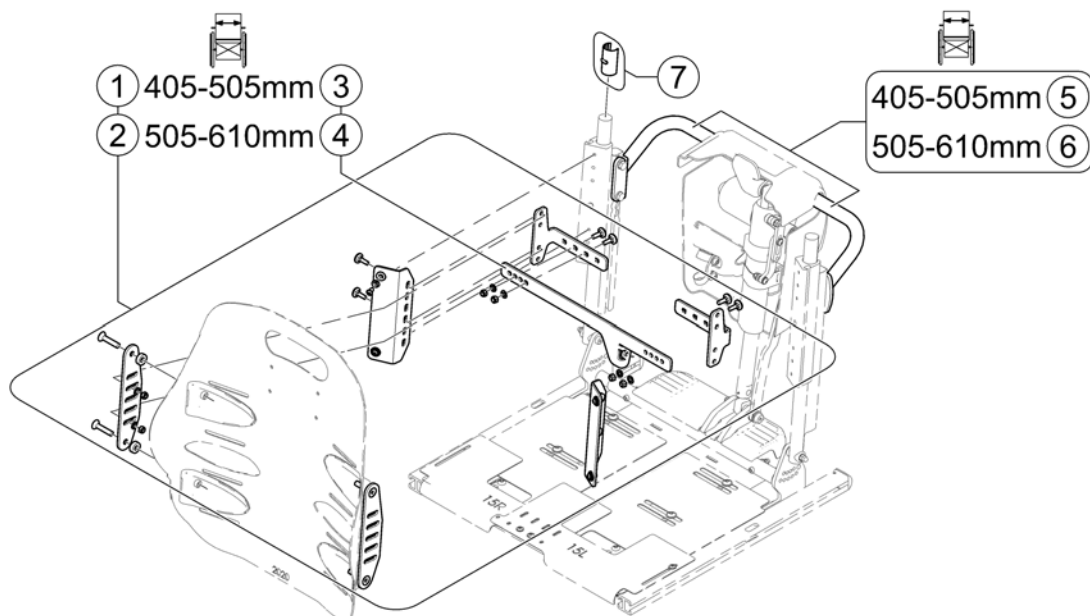

Pos.	Part number	Description	Valid from → until	Qty
1	SPTRC1951	Matrx Back Interface Kit SW 405-505mm (16-20")		1
2	SPTRC2095	Matrx Back Interface Kit SW 505-610mm (19-24")		1
3	SPTRX8299	Matrx Center Plate SW 405-505mm (16-20")		1
4	SPTRX8779	Matrx Center Plate SW 505-610mm (19-24")		1
5	SPTRC2188	Push Bars SW 405-505mm (16-20")		1
6	SPTRC2190	Push Bars SW 505-610mm (19-24")		1
7	SPTRX7883	Linear Bearing		1

Posture Back


158204 Rev

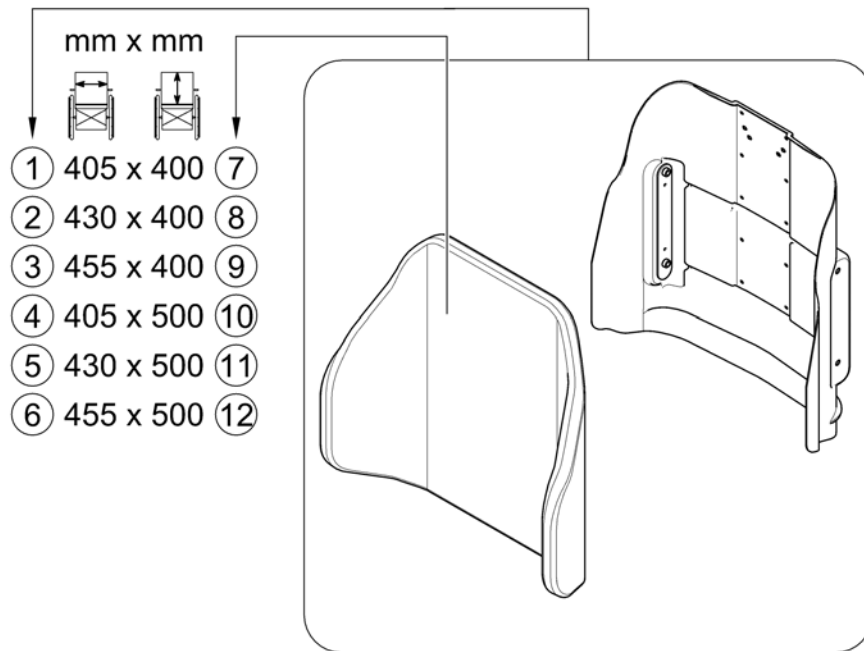Posture Back

Pos.	Part number	Description	Valid from → until	Qty
1	SP1515727	Posture Back 405x400mm (16x16"), cpl.		1
2	SP1515729	Posture Back 430x400mm (17x16"), cpl.		1
3	SP1515731	Posture Back 455x400mm (18x16"), cpl.		1
4	SP1515728	Posture Back 405x500mm (16x20"), cpl.		1
5	SP1515730	Posture Back 430x500mm (17x20"), cpl.		1
6	SP1515732	Posture Back 455x500mm (18x20"), cpl.		1
7	1544896	Backrest Cushion - Posture Back 405x400mm (16x16")		1
8	1544899	Backrest Cushion - Posture Back 430x400mm (17x16")		1
9	1544995	Backrest Cushion - Posture Back 455x400mm (18x16")		1
10	1544897	Backrest Cushion - Posture Back 405x500mm (16x20")		1
11	1544982	Backrest Cushion - Posture Back 430x500mm (17x20")		1
12	1544996	Backrest Cushion - Posture Back 455x500mm (18x20")		1

Elite T/R Back

1580297 Rev

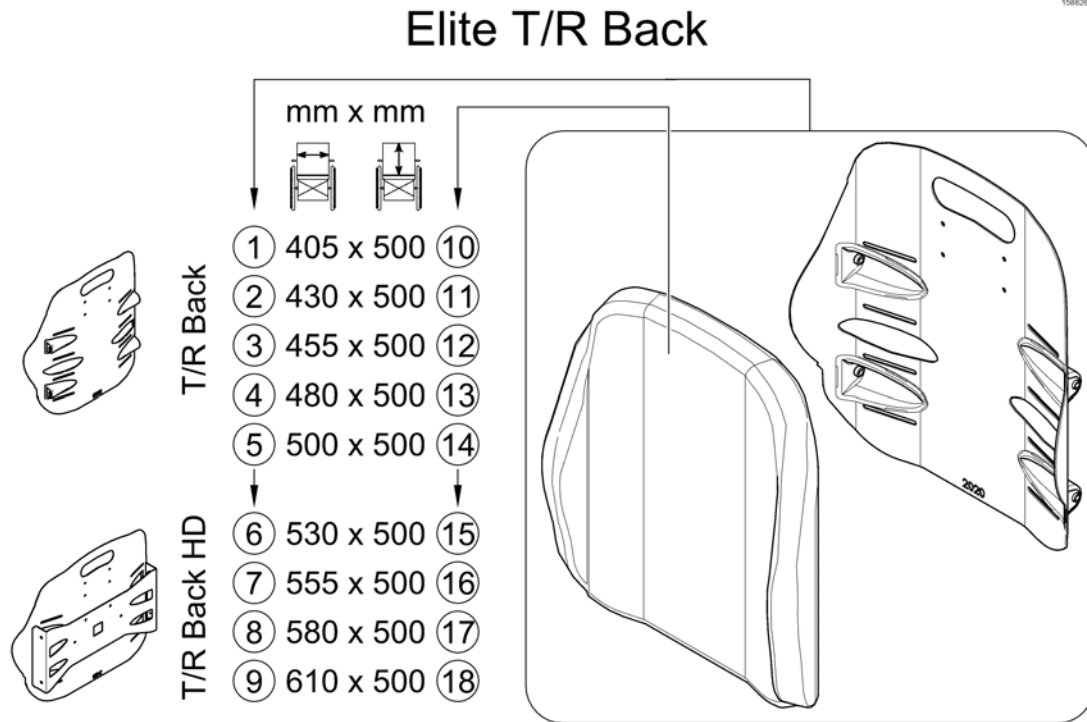


Pos.	Part number	Description	Valid from → until	Qty
1	SP1580029	Elite T/R Back 405x500mm (16x20"), cpl.		1
2	SP1580030	Elite T/R Back 430x500mm (17x20"), cpl.		1
3	SP1580031	Elite T/R Back 455x500mm (18x20"), cpl.		1
4	SP1580032	Elite T/R Back 480x500mm (19x20"), cpl.		1
5	SP1580033	Elite T/R Back 505x500mm (20x20"), cpl.		1
6	SP1584160	Elite T/R Back HD 530x500mm (21x20"), cpl.		1
7	SP1584178	Elite T/R Back HD 555x500mm (22x20"), cpl.		1
8	SP1584180	Elite T/R Back HD 580x500mm (23x20"), cpl.		1
9	SP1584182	Elite T/R Back HD 610x500mm (24x20"), cpl.		1
10	1580110	Backrest Cushion - Elite T/R Back 405x500mm (16x20")		1
11	1580297	Backrest Cushion - Elite T/R Back 430x500mm (17x20")		1
12	1580298	Backrest Cushion - Elite T/R Back 455x500mm (18x20")		1
13	1580299	Backrest Cushion - Elite T/R Back 480x500mm (19x20")		1
14	1580300	Backrest Cushion - Elite T/R Back 505x500mm (20x20")		1
15	SPTRS0375-2120	Backrest Cushion - Elite T/R Back HD 530x500mm (21x20")		1
16	SPTRS0375-2220	Backrest Cushion - Elite T/R Back HD 555x500mm (22x20")		1
17	SPTRS0375-2320	Backrest Cushion - Elite T/R Back HD 580x500mm (23x20")		1
18	SPTRS0375-2420	Backrest Cushion - Elite T/R Back HD 610x500mm (24x20")		1

Elite Deep Back

158208 Rev


Elite Deep Back

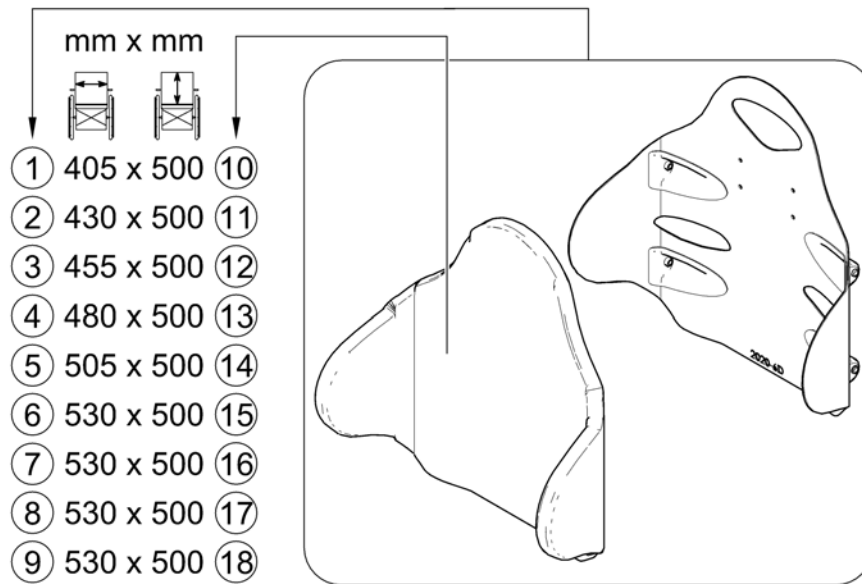

Pos.	Part number	Description	Valid from → until	Qty
1	SP1543515	Elite Deep Back 405x500mm (16x20"), cpl.		1
2	SP1543519	Elite Deep Back 430x500mm (17x20"), cpl.		1
3	SP1543523	Elite Deep Back 455x500mm (18x20"), cpl.		1
4	SP1543987	Elite Deep Back 480x500mm (19x20"), cpl.		1
5	SP1543991	Elite Deep Back 505x500mm (20x20"), cpl.		1
6	SP1584177	Elite Deep Back 530x500mm (21x20"), cpl.		1
7	SP1584179	Elite Deep Back 555x500mm (22x20"), cpl.		1
8	SP1584181	Elite Deep Back 580x500mm (23x20"), cpl.		1
9	SP1583933	Elite Deep Back 610x500mm (24x20"), cpl.		1
10	1544073	Backrest Cushion - Elite Deep Back 405x500mm (16x20")		1
11	1544077	Backrest Cushion - Elite Deep Back 430x500mm (17x20")		1
12	1544081	Backrest Cushion - Elite Deep Back 455x500mm (18x20")		1
13	1544085	Backrest Cushion - Elite Deep Back 480x500mm (19x20")		1
14	1544124	Backrest Cushion - Elite Deep Back 505x500mm (20x20")		1
15	SPTRS0344-2120	Backrest Cushion - Elite Deep Back 530x500mm (21x20")		1
16	SPTRS0344-2220	Backrest Cushion - Elite Deep Back 555x500mm (22x20")		1
17	SPTRS0344-2320	Backrest Cushion - Elite Deep Back 580x500mm (23x20")		1
18	SPTRS0344-2420	Backrest Cushion - Elite Deep Back 610x500mm (24x20")		1

Privacy Flaps

1558496.iss

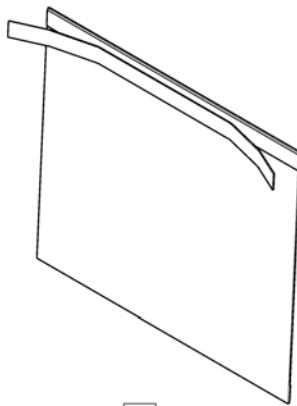

405/430mm	①
455/480mm	②
505/530mm	③

Pos.	Part number	Description	Valid from → until	Qty
1	SP1551800	Privacy Flap SW 405/430mm		1
2	SP1551903	Privacy Flap SW 455/480mm		1
3	SP1551904	Privacy Flap SW 505/530mm		1



Pos.	Part number	Description	Valid from → until	Qty
1	-	Optimist Seats		1
2	-	Push Handle Optimist Seat		1

Optimist Seats


OPTIMIST 1		OPTIMIST 2		OPTIMIST 3		OPTIMIST 5					
1	38cm	46cm	5	38cm	46cm	11	45cm	60cm	13	45cm	70cm
2	45cm	46cm	6	38cm	46cm	12	50cm	70cm	14	50cm	70cm
3	45cm	60cm	7	45cm	46cm						
4	50cm	60cm	8	45cm	60cm						
			9	45cm	60cm						
			10	50cm	60cm						

Arne_Jensen_Sitze.dwg

Pos.	Part number	Description	Valid from → until	Qty
1	1419630	Backrest cushion Optimist Typ1, SW38cm		1
2	5313888	Backrest cushion Optimist Typ1, SW45cm		1
3	5313889	Backrest cushion Optimist Typ1, SW45cm		1
4	1419625	Backrest cushion Optimist Typ1, SW50cm		1
5	1419631	Backrest cushion Optimist Typ2, SW38cm		1
6	5313867	Backrest cushion Optimist Typ2, SW38cm, el. Strap		1
7	5313882	Backrest cushion Optimist Typ2, SW45cm		1
8	5313883	Backrest cushion Optimist Typ2, SW45cm		1
9	5313895	Backrest cushion Optimist Typ2, SW45cm, el. Strap		1
10	1419628	Backrest cushion Optimist Typ2, SW50cm		1
11	5313878	Backrest cushion Optimist Typ3, SW45cm		1
12	5313877	Backrest cushion Optimist Typ3, SW50cm		1
13	5313872	Backrest cushion Optimist Typ5, SW45cm		1
14	5313871	Backrest cushion Optimist Typ5, SW50cm		1

Invacare® Storm4 | Backrest
Push Handle Optimist Seat

1441556X.dwg

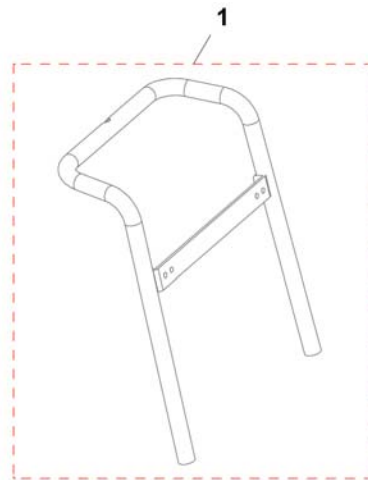


Pos.	Part number	Description	Valid from → until	Qty
1	1441553	Push Handle Arne Jensen		1

Storm4 TTPlus

Pos.	Part number	Description	Valid from → until	Qty
1	-	Backrest for Standard Seat		1
2	-	Backrest for EasyAdapt Seat		1
3	-	Backrest for Modulite Seat		0
4	-	Backrest for Optimist Seat		1

Backrest for Modulite Seat


1586095.kso

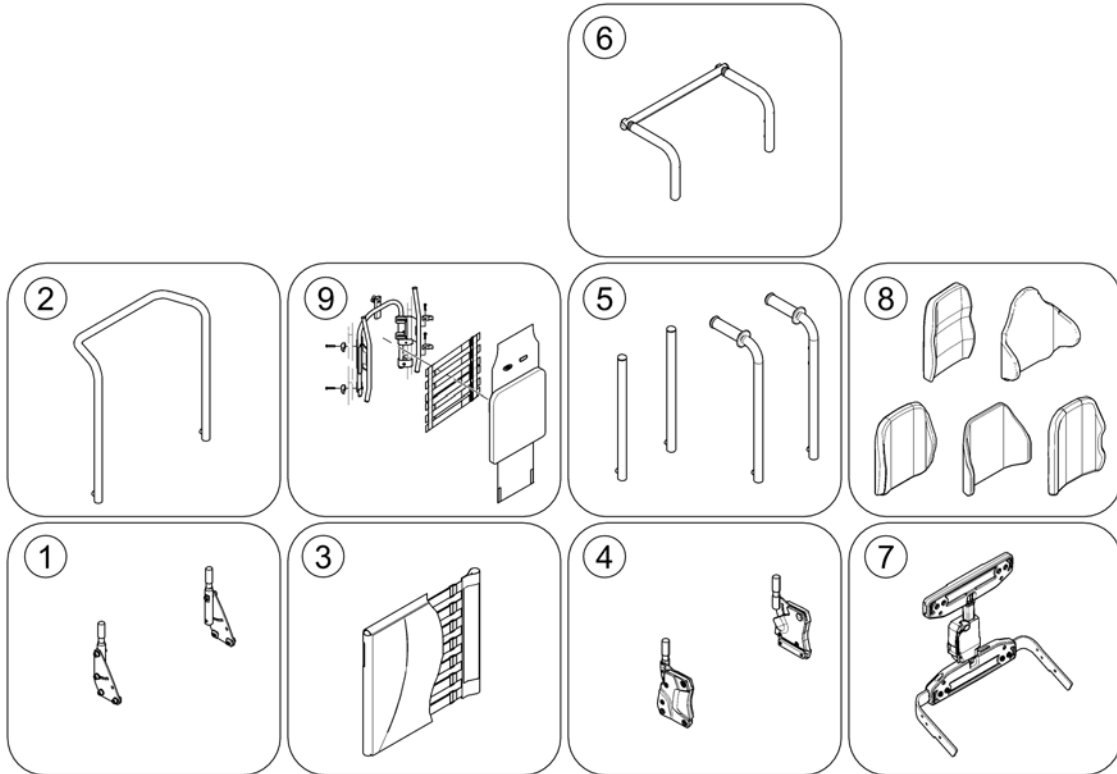

Pos.	Part number	Description	Valid from → until	Qty
1	-	Support for Backrest Frame <i>[0975+2161]</i>		1
2	-	Backrest Frame <i>[2161]</i>		1
3	-	Sling Back, tension adjustable <i>[2171]</i>		1
4	-	Backrest support		1
5	-	Backposts <i>[2162/2163]</i>		1
6	-	Connection tube <i>[1307]</i>		1
7	-	Backrest Recliner / Stabilisation bracket <i>[0977]</i>		1
8	-	Backrest Cushions <i>[2172/2173/2174/2175/2176/2177/2178]</i>		1
9	-	Flex3 back, tension adjustable <i>[2179]</i>		1

Backrest Recliner / Stabilisation bracket

1583079.iss

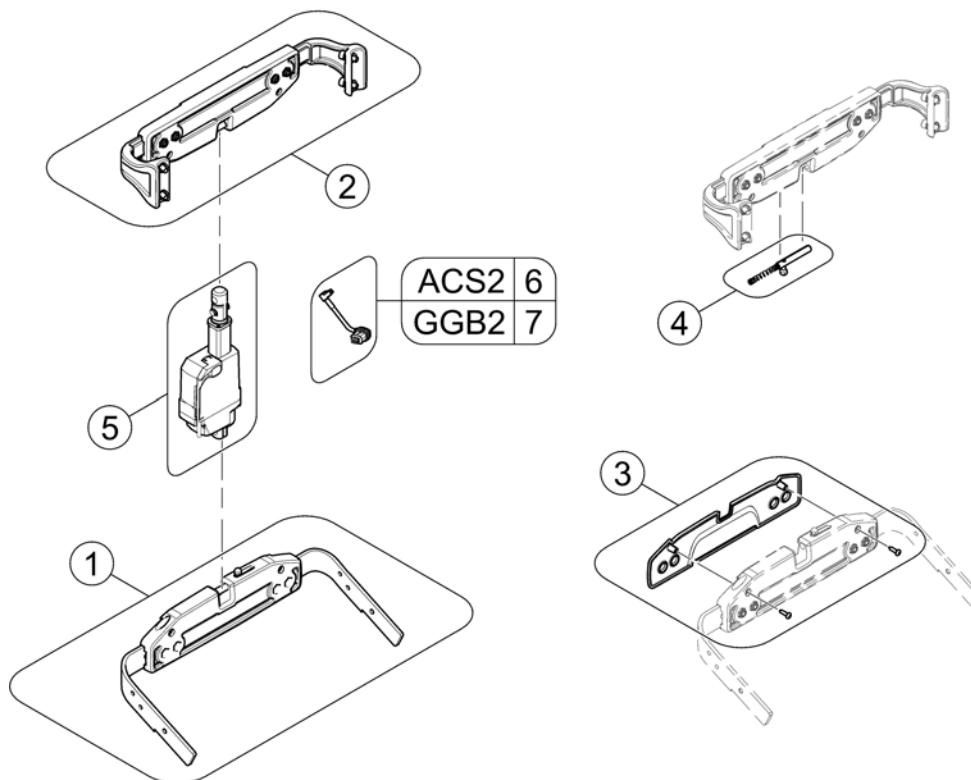

Pos.	Part number	Description	Valid from → until	Qty
1	SP1567169	Actuator bracket (pre set to SW 43cm)		1
2	SP1567168	Stabilization bracket (pre set to SW 43cm)		1
3	SP1571702	Cover		1
4	SP1571704	Locking bolt		1
5	SP1557820	Actuator LA23		1
6	SP1572166	Actuator Cable LA23 ACS2 (~950mm)		1
7	SP1579551	Actuator Cable LA23 GGB2 (~850mm)		1

Storm4

Pos.	Part number	Description	Valid from → until	Qty
-		Legrests for Ultra Low Maxx Seat		1
-		Legrest Support, Vari series 07/2005 ==>		0
-		Hanger Vari A (04/2009 =>)		1
-		Hanger Vari F		0
-		Hangers ADM LH/RH manual		0
-		Hanger Vari ADE/ADP electric (07/2005 =>)		0
-		Central Legrest, manuell		0
-		Central Legrest, electric		0
-		Footrests Vari, Flip-Up (01.08.10 =>)		1
-		Footrests Vari, Flip-Up & angle adjustable (01.08.10 =>)		1
-		Support Tube with Footplate for central Legrest		1
-		Lower Footrests		1

Legrests for Ultra Low Maxx Seat


Pos.	Part number	Description	Valid from → until	Qty
-		Central Legrests		1
-		Vari Legrests		1
-		Pivot Plus Legrests		1

Central Legrests


Pos.	Part number	Description	Valid from → until	Qty
-		Supports, Central Legrests		1
-		Central Legrest pow. (LNX)		1
-		Central Legrest man.		1

Supports, Central Legrests

158408 Rev

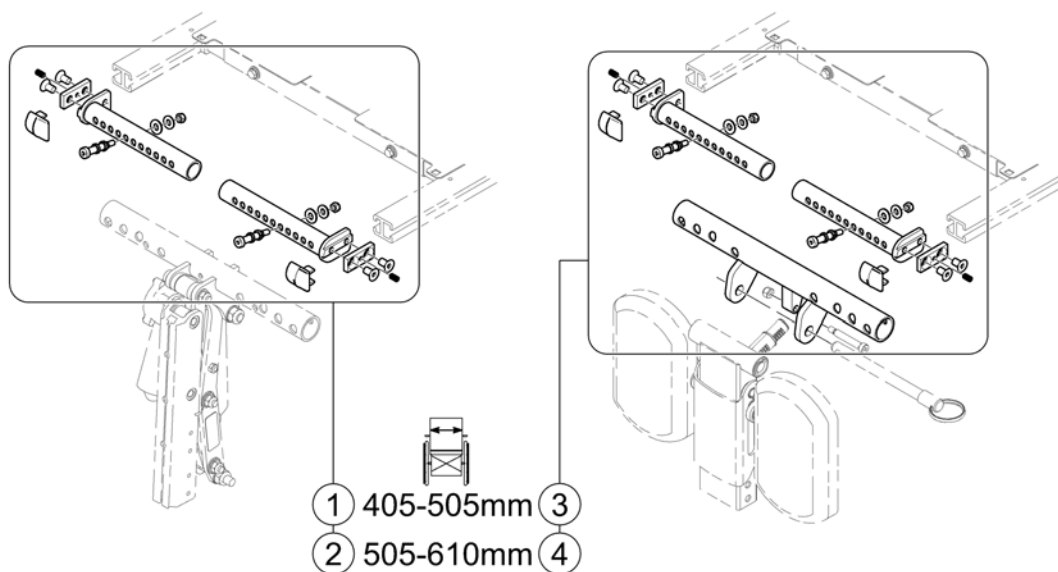


Pos.	Part number	Description	Valid from → until	Qty
1	SPTRC1945	Legrest Support, el. - SW 405-505mm (16-20")		1
2	SPTRC2096	Legrest Support, el. - SW 505-610mm (19-24")		1
3	SPTRC1947	Legrest Support, man. - SW 405-505mm (16-20")		1
4	SPTRC2097	Legrest Support, man. - SW 505-610mm (19-24")		1

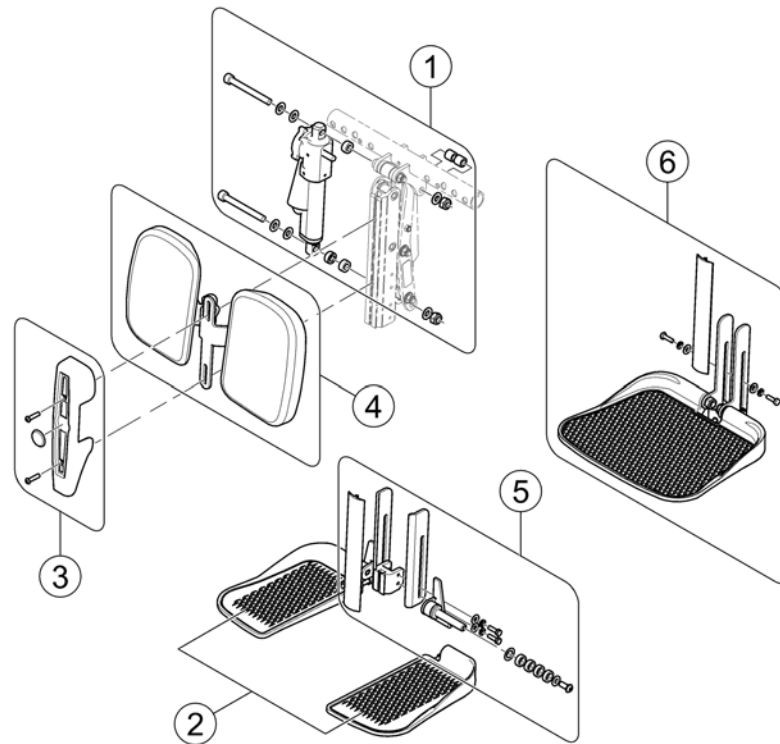

Pos.	Part number	Description	Valid from → until	Qty
-	SP1602775	Central Legrest LNX, powered, cpl. (with small footplates)		1
-	SPTRE0756	Actuator cable		1
1	SPTRC2124	Legrest actuator, cpl. with mounting hardware		1
2	SPTRC2125	Foot plates 240mm depth (pair), cpl.		1
2	SPTRC2125_C03	Foot plates 290mm depth (pair), cpl.	2016-01-01 →	1
3	SPTRC2126	Cover, cpl. with mounting hardware		1
4	SP1602681	Calf pads, cpl. with mounting hardware		1
5	SPTRC1664-C01	Mounting hardware, foot plates		1
6	SPTRC2222	Footboard 305x240mm, cpl. with mounting hardware	2016-01-01 →	1

Central Legrest man.

153396.km

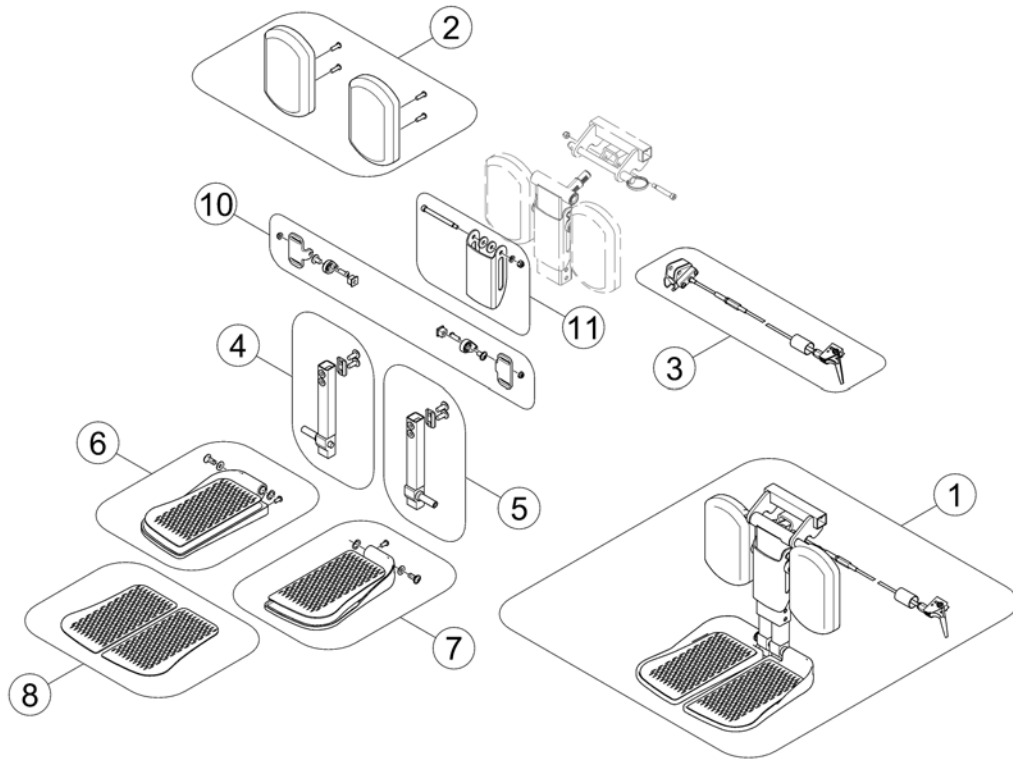


Pos.	Part number	Description	Valid from → until	Qty
1	SP1537926	Central Legrest manual, cpl.		1
2	SP1163373	Calfplates (Pair)		1
3	SP1121079	Bowden cable central legrest, manual		1
4	SP1161986	Kit, extension tube, cpl. RH		1
5	SP1161987	Kit, extension tube, cpl. LH		1
6	SP1161990	Footplate, cpl. RH		1
7	SP1161991	Footplate, cpl. LH		1
8	SP1163375	Footplate pads (pair)		1
10	SP1165505	Kit, Adjustment Hardware Calf Pads		1
11	SP1165506	Kit, Calf Pad mounting bracket w/ hardware		1

Vari Legrests


Pos.	Part number	Description	Valid from → until	Qty
-		Supports, Vari		1
-		Hanger Vari A (04/2009 =>)		1
-		Hanger Vari F		1
-		Hangers ADM LH/RH manual		1
-		Hanger Vari ADE/ADP electric (07/2005 =>)		1
-		Footrests Vari, Flip-Up (01.08.10 =>)		1
-		Footrests Vari, Flip-Up & angle adjustable (01.08.10 =>)		1
-		Footrests Vari, angle & depth adjustable		1

Supports, Vari

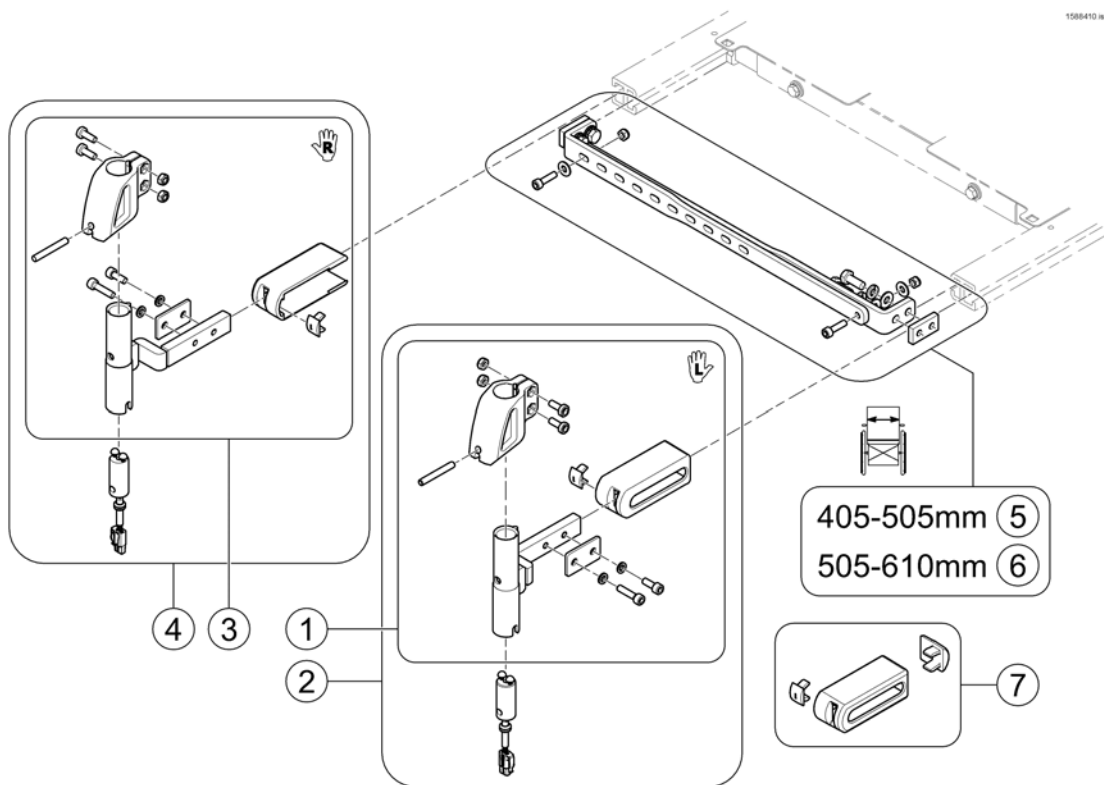

Pos.	Part number	Description	Valid from → until	Qty
1	SP1586651	Legrest Support Vari LH man cpl.		1
2	SP1587187	Legrest Support Vari LH el. cpl.		1
3	SP1586706	Legrest Support Vari RH man cpl.		1
4	SP1587186	Legrest Support Vari RH el. cpl.		1
5	SPTRC2098	Cross brace SW 405-505mm (16-20")		1
6	SPTRC2099	Cross brace SW 505-610mm (19-24")		1
7	SPTRC2061	End cap kit		1


Hanger Vari A (04/2009 =>)

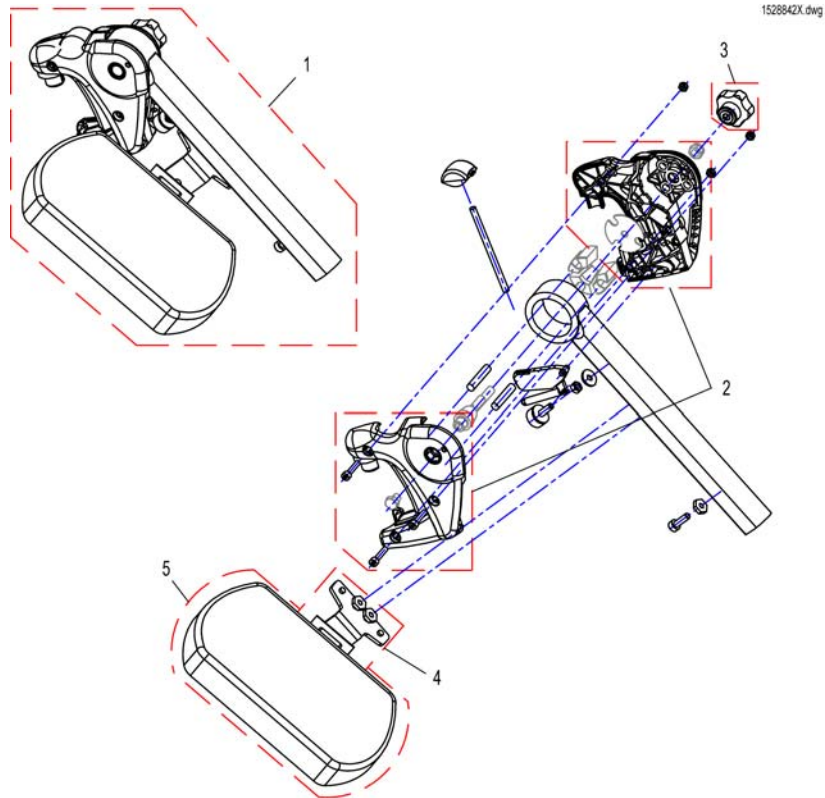

Pos.	Part number	Description	Valid from → until	Qty
-	SP1527820	Rework-Kit Vari A Legrest Version 1 => Version 2 <i>Vari A Version 1 => Version 2 Upgrade Kit</i>		1
1	SP1527498	Hanger Vari A LH, cpl.		1
1	SP1527492	Hanger Vari A RH, cpl.		1
2	SP1434955	Cover Legrest cpl.		1
3	L34011	Handknob M8 Tristarfemale		1
4	SP1450447	Calf Pad Holder Left Turnable, cpl.		1
4	SP1450446	Calf Pad Holder Right Turnable, cpl.		1
5	6301333	Legrest calf pad		1

Hanger Vari F


144934 / X.dwg

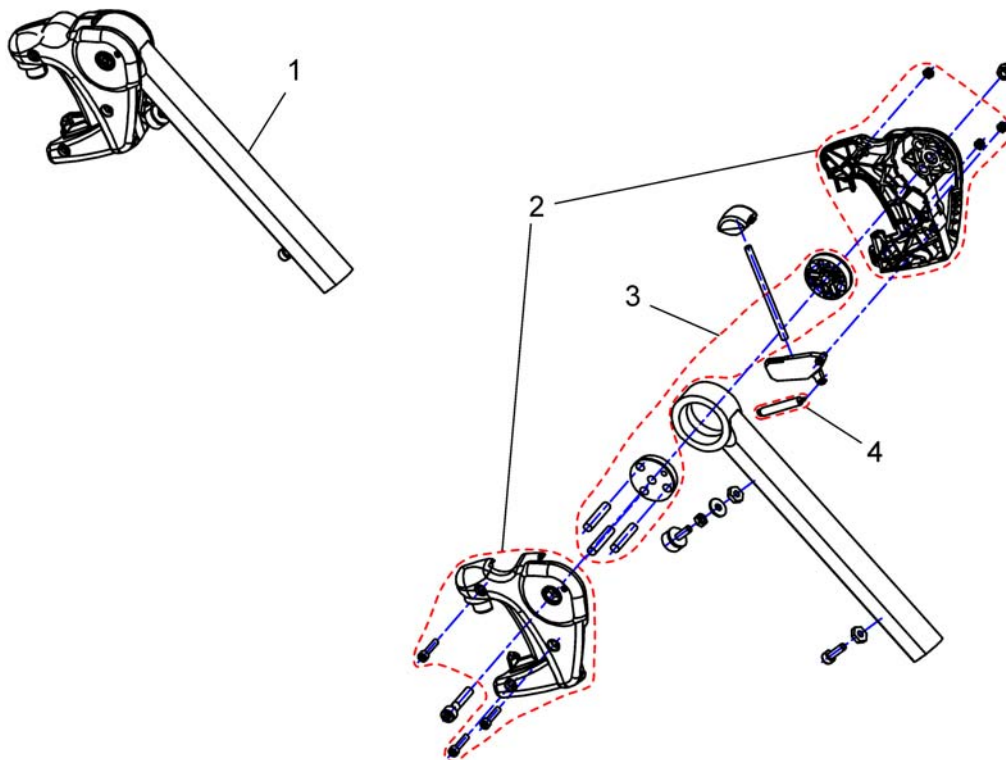

Pos.	Part number	Description	Valid from → until	Qty
1	SP1449347	Hanger Vari F LH cpl.		1
1	SP1449346	Hanger Vari F RH cpl.		1
2	SP1434955	Cover Legrest cpl.		1
3	1434956	Mechanic		1
4	SPZ173178	PULL SPRING		1

Hangers ADM LH/RH manual


1449409X.dwg

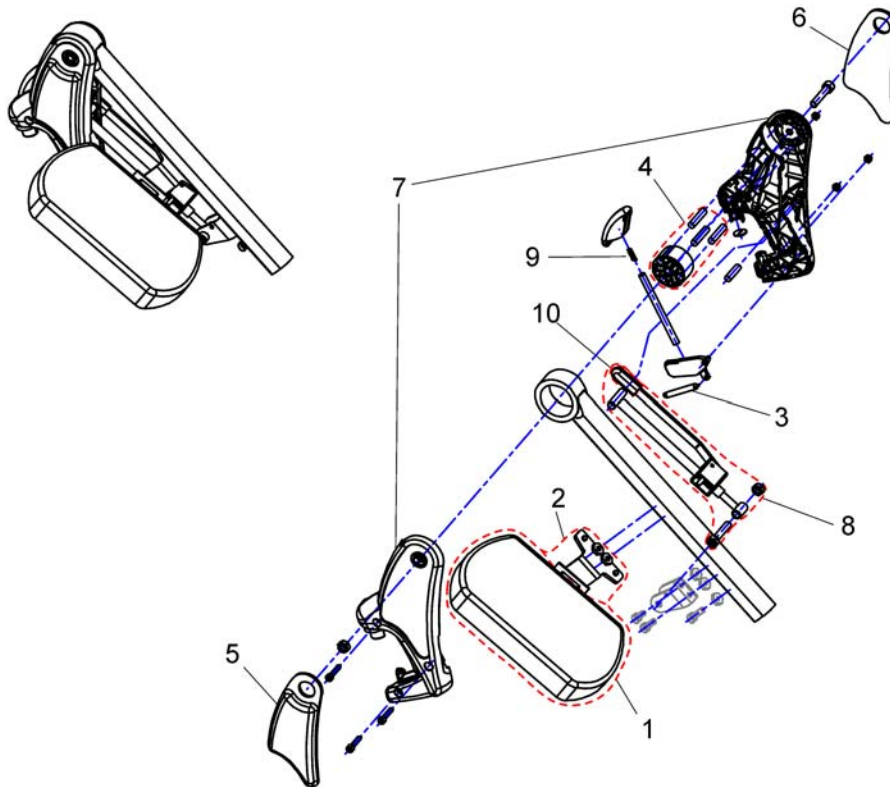

Pos.	Part number	Description	Valid from → until	Qty
1	6301333	Legrest calf pad		1
2	SP1450447	Calf Pad Holder Left Turnable, cpl.		1
2	SP1450446	Calf Pad Holder Right Turnable, cpl.		1
3	SPZ173178	PULL SPRING		1
4	SPZ174134	SLIDING BUSHING		1
5	SP1450410	Knee Upholstery LH		1
6	SP1450448	Knee Upholstery RH		1
7	SP1449293	Ad-Cover cpl.		1
8	SPZ174130	ADM Adjustment mechanism cpl.		1
9	SPN12196	COMPRESSION SPRING L0=16, 4 D=0, 5 DA=5, 5		1
10	SP1006490	Brake Lever Rubber -LT2000		1
11	SP1449410	Hanger ADM RH cpl.		1
11	SP1449409	Hanger ADM LH cpl.		1

Hanger Vari ADE/ADP electric (07/2005 =>)


1449407X.dwg

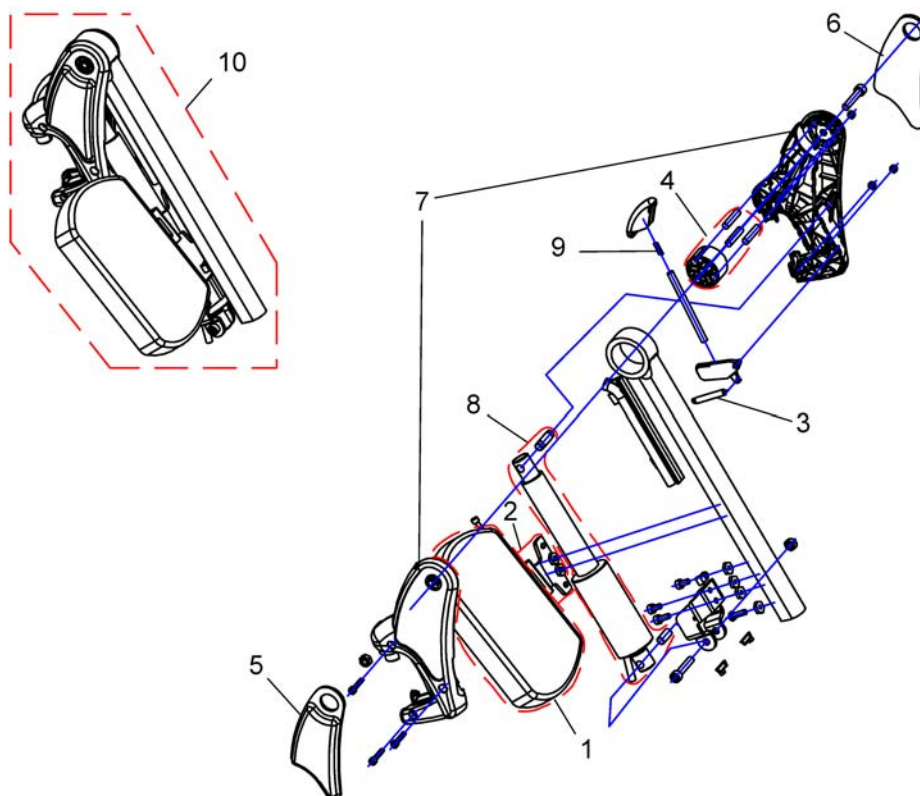

Pos.	Part number	Description	Valid from → until	Qty
1	6301333	Legrest calf pad		1
2	SP1450447	Calf Pad Holder Left Turnable, cpl.		1
2	SP1450446	Calf Pad Holder Right Turnable, cpl.		1
3	SPZ173178	PULL SPRING		1
4	SPZ174134	SLIDING BUSHING		1
5	SP1450448	Knee Upholstery RH		1
6	SP1450410	Knee Upholstery LH		1
7	SP1449293	Ad-Cover cpl.		1
8	SPZ151006	Actuator with circuit board, preassembled		1
9	SPN12196	COMPRESSION SPRING L0=16, 4 D=0, 5 DA=5, 5		1
10	SP1449408	Hanger ADP RH cpl.		1
10	SP1449407	Hanger ADP LH cpl.		1

Footrests Vari, Flip-Up (01.08.10 =>)


152964X.lin

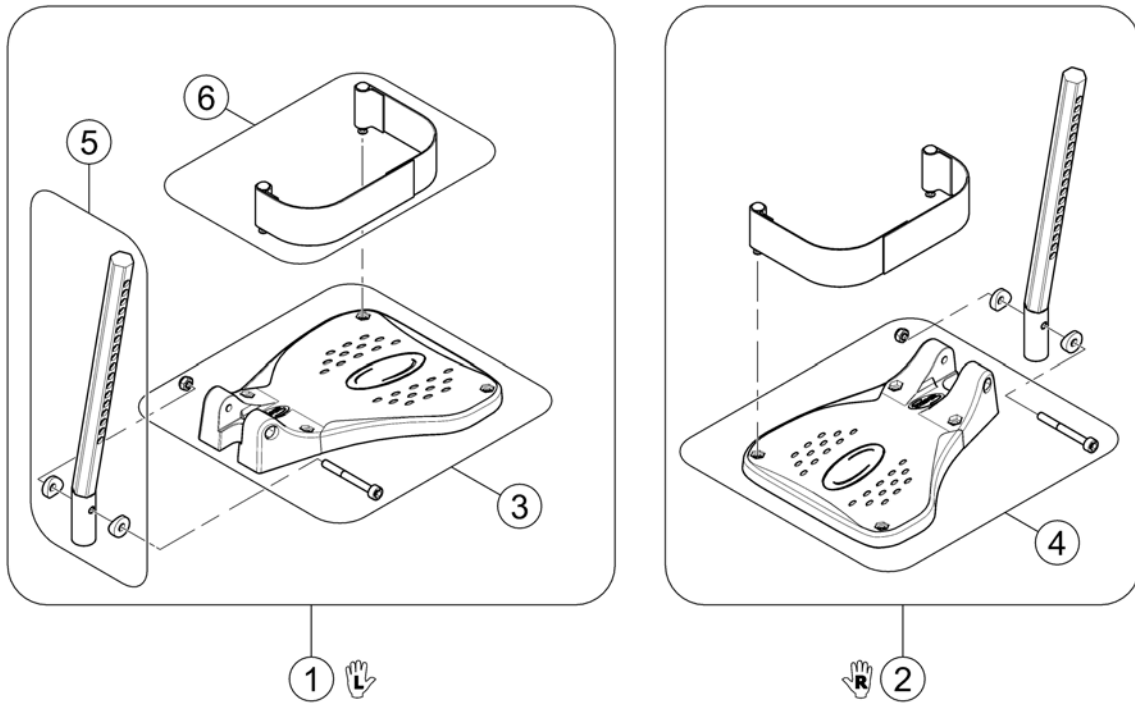

Pos.	Part number	Description	Valid from → until	Qty
1	SP1541575	Footplate flip up, LH, cpl.		1
2	SP1541574	Footplate flip up, RH, cpl.		1
3	SP1529645	FOOTPLATE LH		1
4	SP1529644	FOOTPLATE RH		1
5	SP1543045	Plug-in tube LH/RH		1
6	SP1442467	Heel Strap cpl.		1

Footrests Vari, Flip-Up & angle adjustable (01.08.10 =>)


152964EX.es

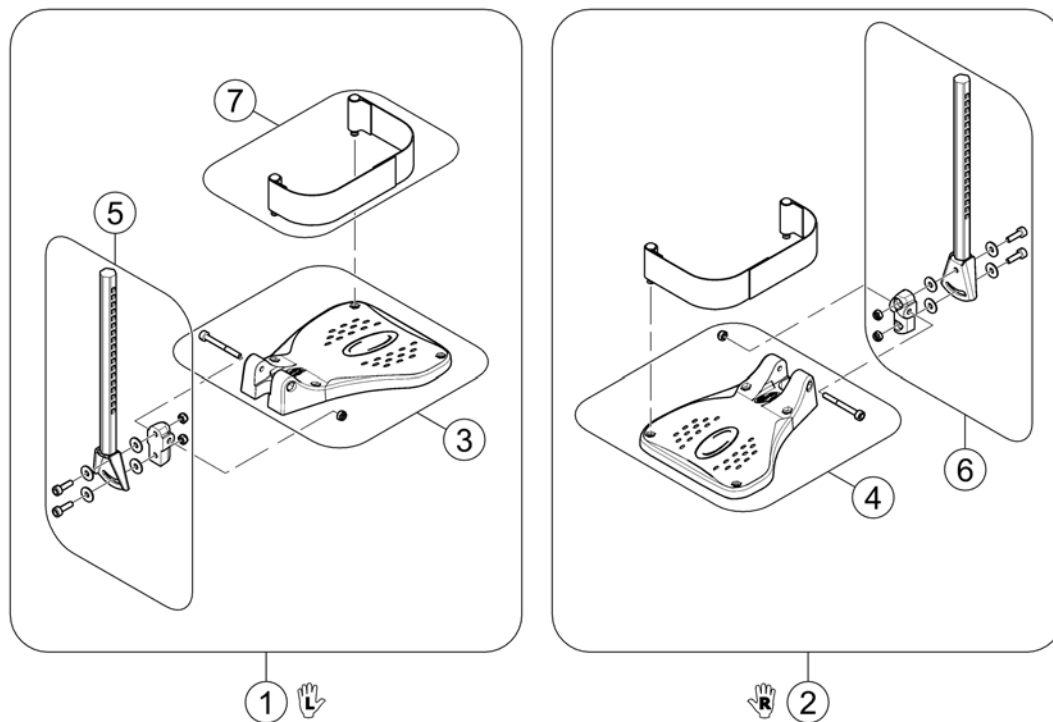

Pos.	Part number	Description	Valid from → until	Qty
1	SP1541580	Footplate flip up, angle adjustable, LH, cpl.		1
2	SP1541579	Footplate flip up, angle adjustable, RH, cpl.		1
3	SP1529645	FOOTPLATE LH		1
4	SP1529644	FOOTPLATE RH		1
5	SP1423204	FOOT PLATE TUBE, ANGLE ADJUSTMENT LH CPL.		1
6	SP1423203	FOOT PLATE TUBE, ANGLE ADJUSTMENT RH CPL.		1
7	SP1442467	Heel Strap cpl.		1

Footrests Vari, angle & depth adjustable


1583898 iso

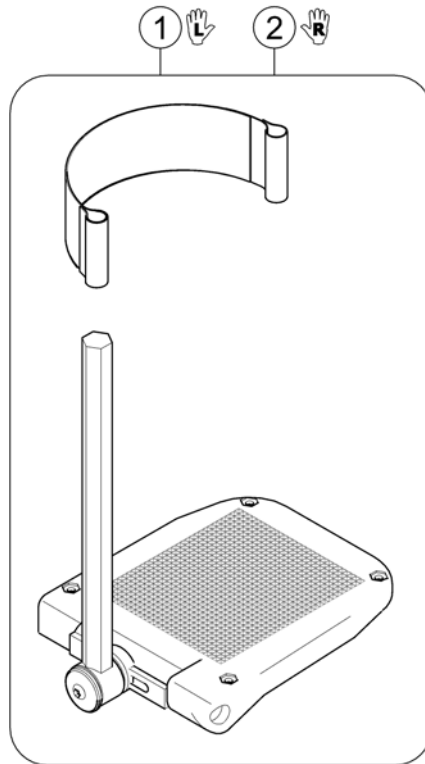

Pos.	Part number	Description	Valid from → until	Qty
1	SP1419344	Footplate LH Anlge And Depth Adjustable, With Support Tube cpl.		1
2	SP1419345	Footplate RH Anlge And Depth Adjustable, With Support Tube cpl.		1

Pivot Plus Legrests


Pos.	Part number	Description	Valid from → until	Qty
-		Cross Brace, Pivot Plus		1
-		Legrests		1

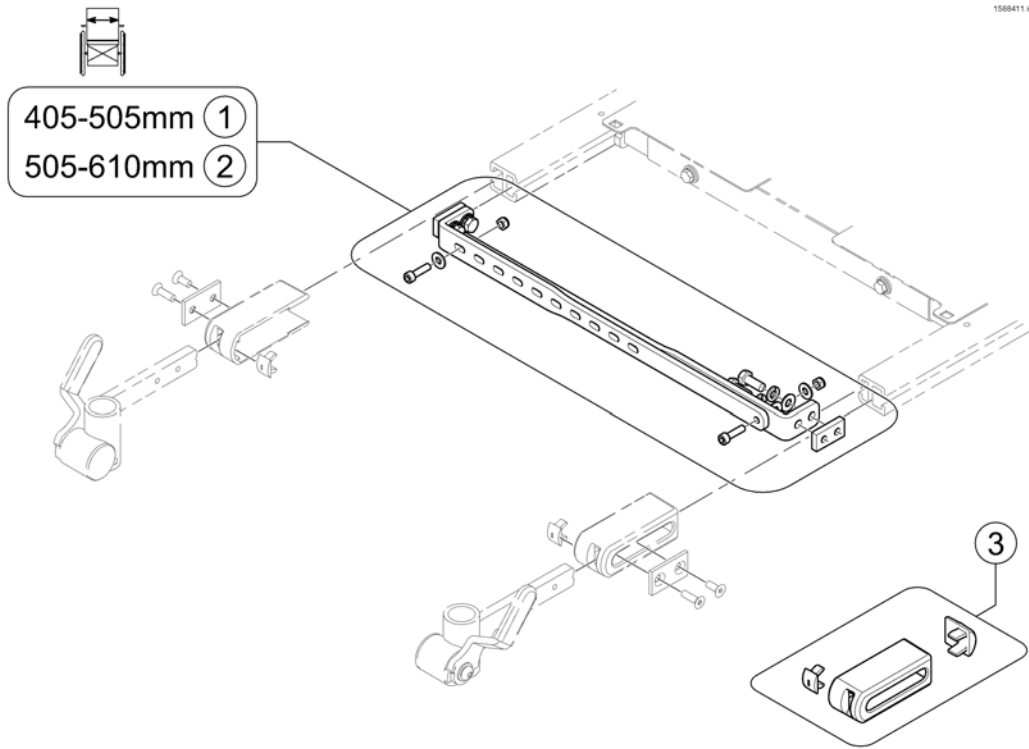

Pos.	Part number	Description	Valid from → until	Qty
1	SPTRC2098	Cross brace SW 405-505mm (16-20")		1
2	SPTRC2099	Cross brace SW 505-610mm (19-24")		1
3	SPTRC2061	End cap kit		1

Legrests


1588425.iss

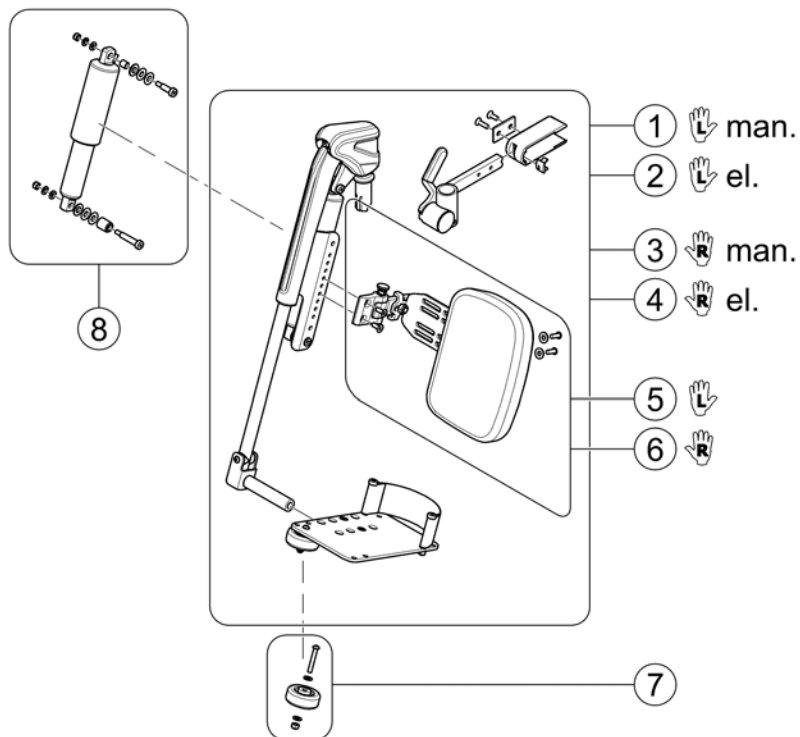

Pos.	Part number	Description	Valid from → until	Qty
1	SPTRC2070	Pivot Plus Legrest man. LH, cpl.		1
2	SPTRC2068	Pivot Plus Legrest el. LH, cpl.		1
3	SPTRC2069	Pivot Plus Legrest man. RH, cpl.		1
4	SPTRC2067	Pivot Plus Legrest el. RH, cpl.		1
5	SPTRC2127-C02	Calf Pad LH with mounting hardware		1
6	SPTRC2127-C01	Calf Pad RH with mounting hardware		1
7	SPTRC2129	Wheel, Footplate - cpl.		1
8	SPTRC2128	Legrest Actuator, cpl.		1
-	SPTRE0756	Actuator cable <i>LH / links</i>		1
-	SPTRE0754	Actuator Cable <i>RH / rechts</i>		1

Legrest Support, Vari series 07/2005 ==>


1449734 iso

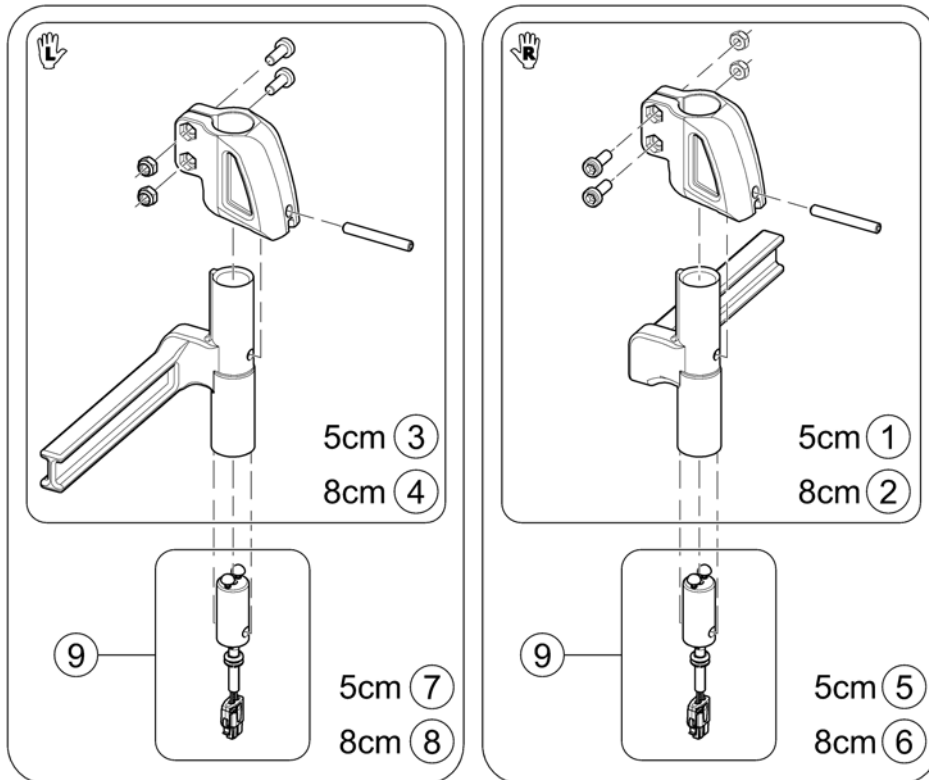

Pos.	Part number	Description	Valid from → until	Qty
1	SP1449732	Legrest Support Vari series, man. 5cm - RH		1
2	SP1449736	Footrest/Legrest receptacle, manuell, RH, 8cm		1
3	SP1449733	Legrest Support Vari series, man. 5cm - LH		1
4	SP1449737	Footrest/Legrest receptacle, manuell, LH, 8cm		1
5	SP1531071	Legrest Support Vari series, powered 5cm - RH		1
6	SP1531073	Legrest Support Vari series, powered 8cm - RH		1
7	SP1531072	Legrest Support Vari series, powered 5cm - LH		1
8	SP1531082	Legrest Support Vari series, powered 8cm - LH		1
9	SP1530795	Power Supply Cable Legrest ACS2 (1000mm)		1


Central Legrest, manuell

1529387X.dwg

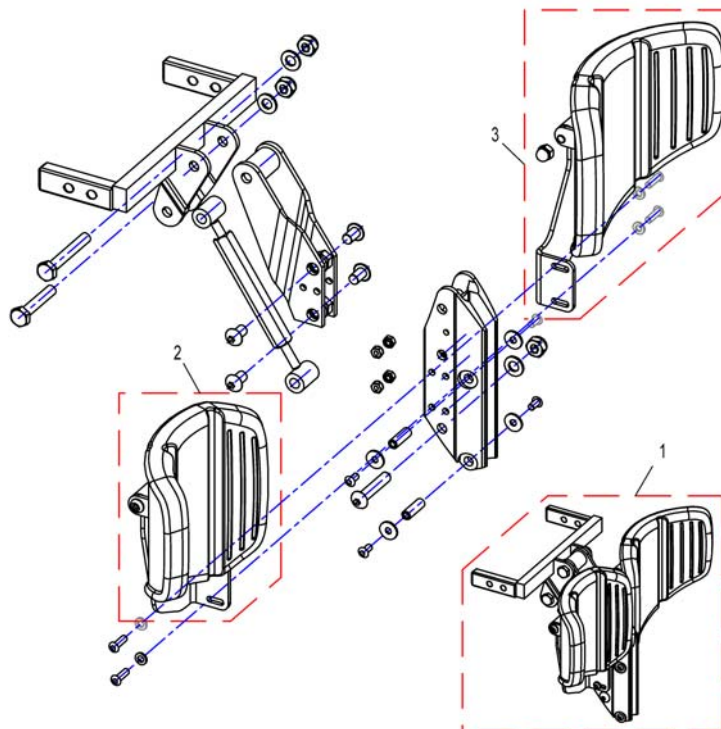

Pos.	Part number	Description	Valid from → until	Qty
1	SP1529387	Central Legrest, manuell, cpl.		1
2	1436070	CALF PLATE RH, COMPLETE WITH HOLDER		1
3	1436069	CALF PLATE LH, COMPLETE WITH HOLDER		1


152938X.iss

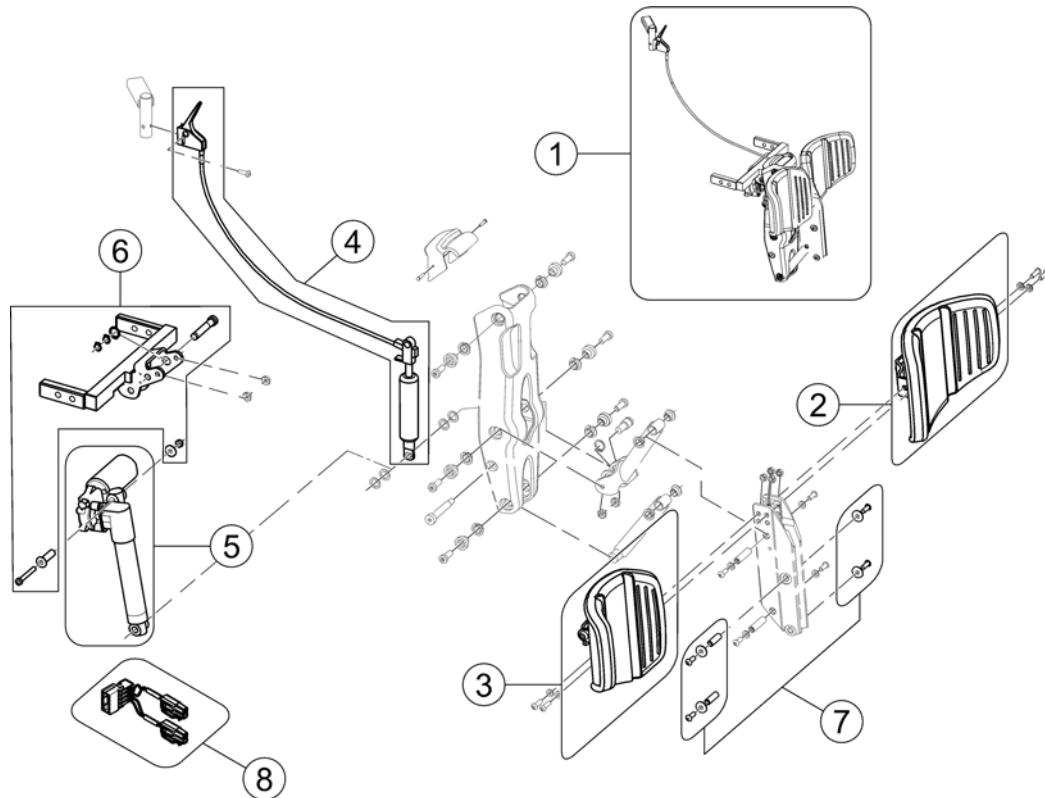

Pos.	Part number	Description	Valid from → until	Qty
1	SP1529388	Central Legrest, electric, complete		1
2	1436076	Calf Plate with holder LH		1
3	1440594	Calf Plate with holder RH		1
4	1440593	Gas Pressure Spring		1
5	SP1440600	Actuator el. Central Legrest 90°		1
6	SP1574298	Bracket for el. Central Legrest		1
7	SP1576334	Mounting Hardware Arm 2-E		1
8	SP1524744	Adaptercable Actuator AMP -> DX2		1


Support Tube with Footplate for central Legrest

1440290X.dwg

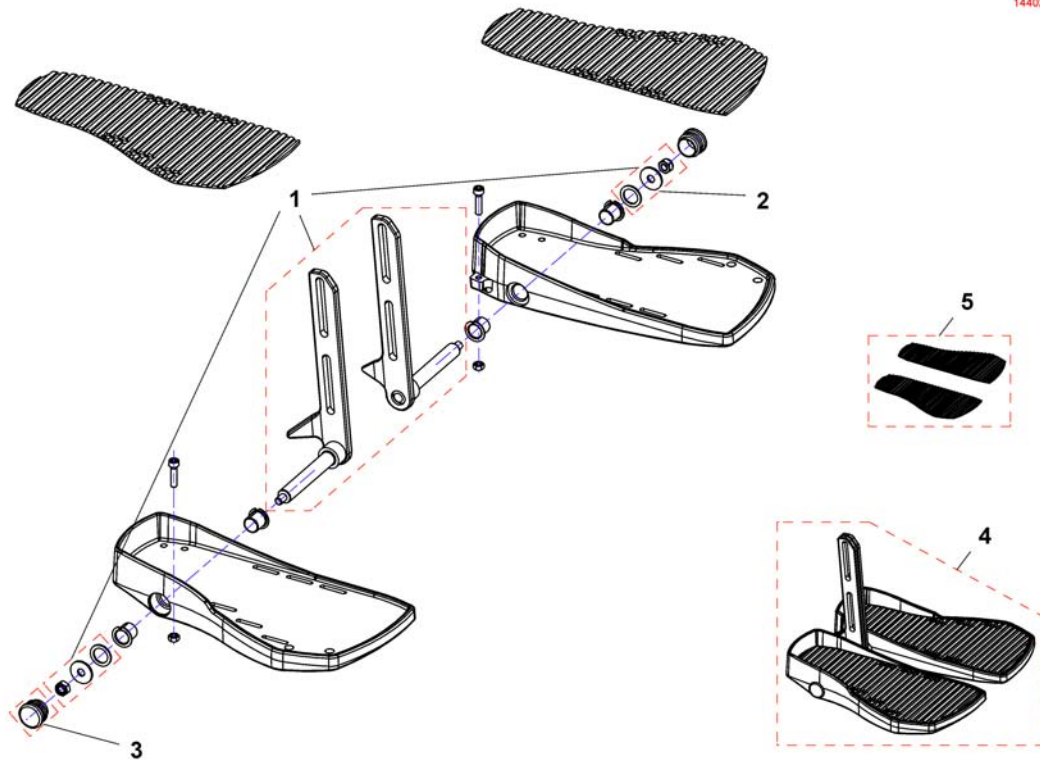

Pos.	Part number	Description	Valid from → until	Qty
1	SP1440442	Central Lower Legrest, small		1
1	SP1440443	Central Lower Legrest, medium		1
1	SP1440444	Central Lower Legrest, large		1
2	SPF00002	Hexagon Nut M8 bk - DIN 985		1
3	SP1440517	Darning		1
4	SP1440290	Central Lower Legrest, small (32-38cm), cpl.		1
4	SP1440291	Central Lower Legrest, medium (38-44cm), cpl.		1
4	SP1440292	Central Lower Legrest, large (44-50cm), cpl.		1
5	SP1546720	Footboardrubber		1

Lower Footrests

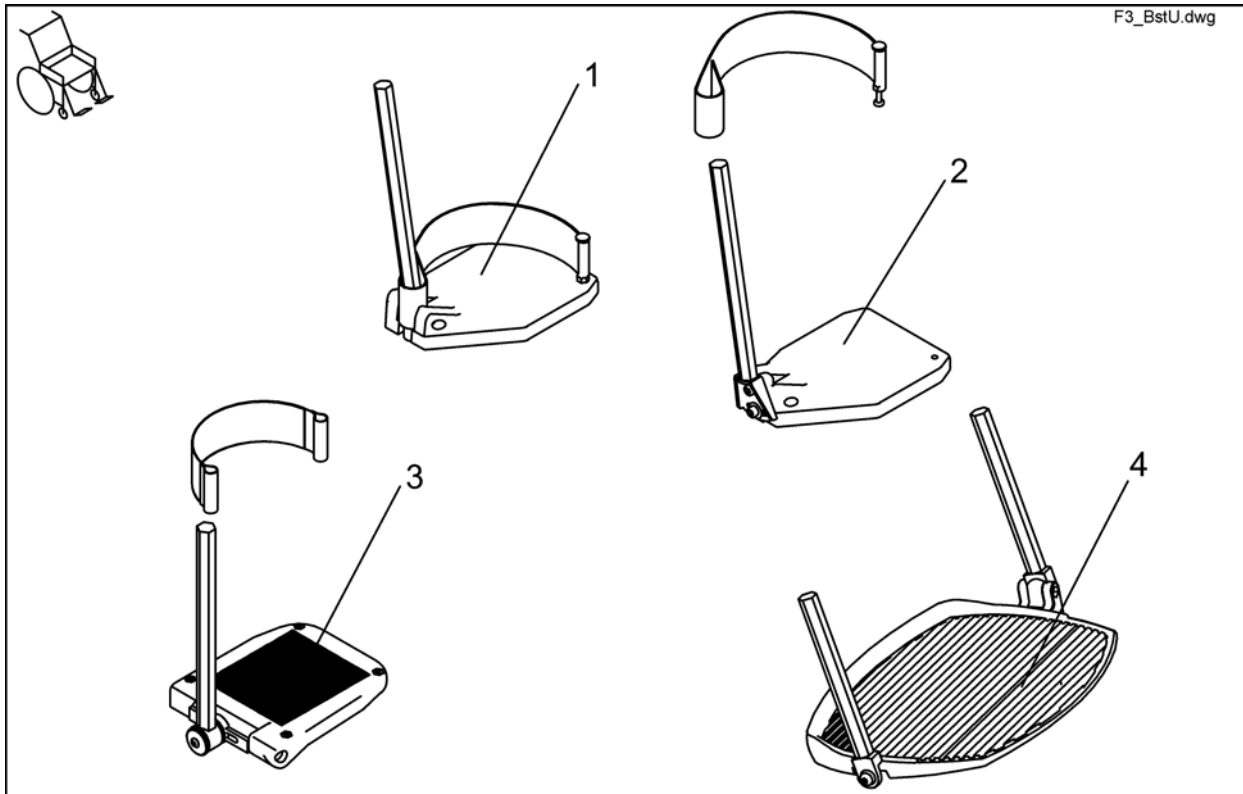


Pos.	Part number	Description	Valid from → until	Qty
1	SP1421662	Footplate RH fix, with Support Tube cpl.		1
1	SP1421663	Footplate LH fix, with Support Tube cpl.		1
2	SP1421664	Footplate RH anlge adjustable, with Support Tube cpl.		1
2	SP1421665	Footplate LH anlge adjustable, with Support Tube cpl.		1
3	SP1419344	Footplate LH Anlge And Depth Adjustable, With Support Tube cpl.		1
3	SP1419345	Footplate RH Anlge And Depth Adjustable, With Support Tube cpl.		1
4	SP1423156	Footboard, pre-assembled		1
4	SP1533540	Footboard		1

Storm4 TTPlus

Pos.	Part number	Description	Valid from → until	Qty
-		Legrest Supports Vari (07/2005 =>)		0
-		Hanger Vari A (04/2009 =>)		0
-		Hanger Vari F		0
-		Hangers ADM LH/RH manual		0
-		Hanger Vari ADE/ADP electric (07/2005 =>)		0
-		Central Legrest, manuell		0
-		Central Legrest, electric		0
-		Footrests Vari, Flip-Up (01.08.10 =>)		1
-		Footrests Vari, Flip-Up & angle adjustable (01.08.10 =>)		1
-		Support Tube with Footplate for central Legrest		1
-		Lower Footrests		1

Legrest Supports Vari (07/2005 =>)


1582374.00

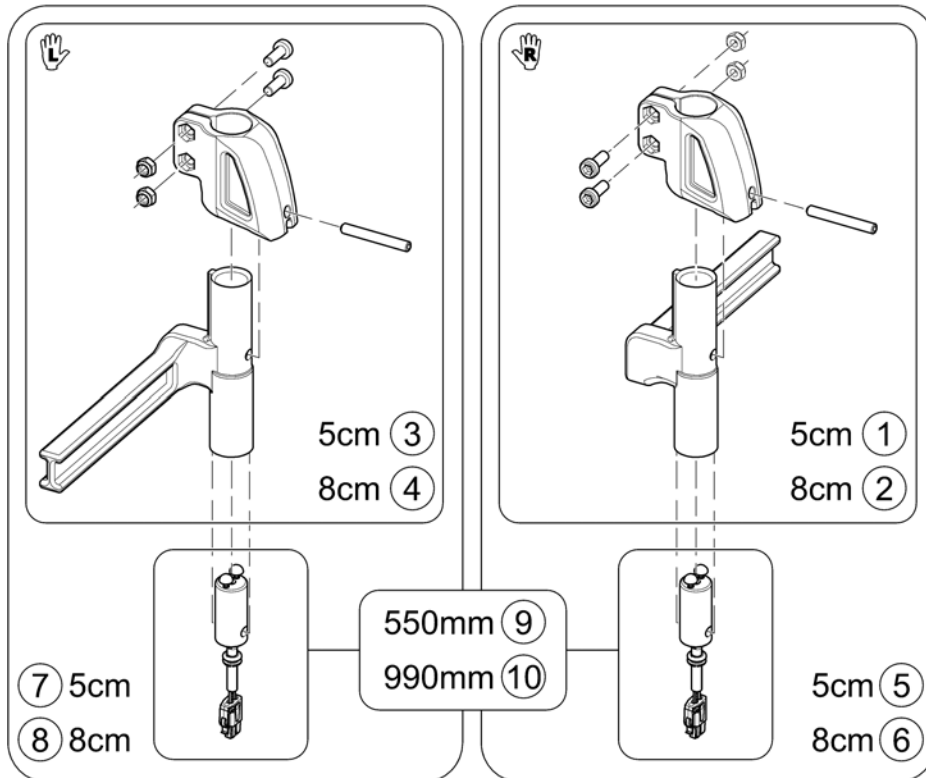

Pos.	Part number	Description	Valid from → until	Qty
1	SP1449732	Legrest Support Vari series, man. 5cm - RH		1
2	SP1449736	Footrest/Legrest receptacle, manuell, RH, 8cm		1
3	SP1449733	Legrest Support Vari series, man. 5cm - LH		1
4	SP1449737	Footrest/Legrest receptacle, manuell, LH, 8cm		1
5	SP1449314	Footrest/Legrest receptacle, powered, RH, 5cm		1
6	SP1449315	Footrest/Legrest receptacle, powered, RH, 8cm <i>EasyAdapt</i>		1
7	SP1449735	Footrest/Legrest receptable, powered, LH, 5cm <i>EasyAdapt</i>		1
7	SP1537398	Footrest/Legrest receptacle, powered, LH, 5cm <i>Modulite</i>		1
8	SP1449739	Footrest/Legrest receptacle, powered, LH, 8cm <i>EasyAdapt</i>		1
9	SP1450232	Power Supply Cable Legrest (550mm)		1
10	SP1450228	Power Supply Cable Legrest (990mm)		1

Central Legrest, electric

1583085 iso

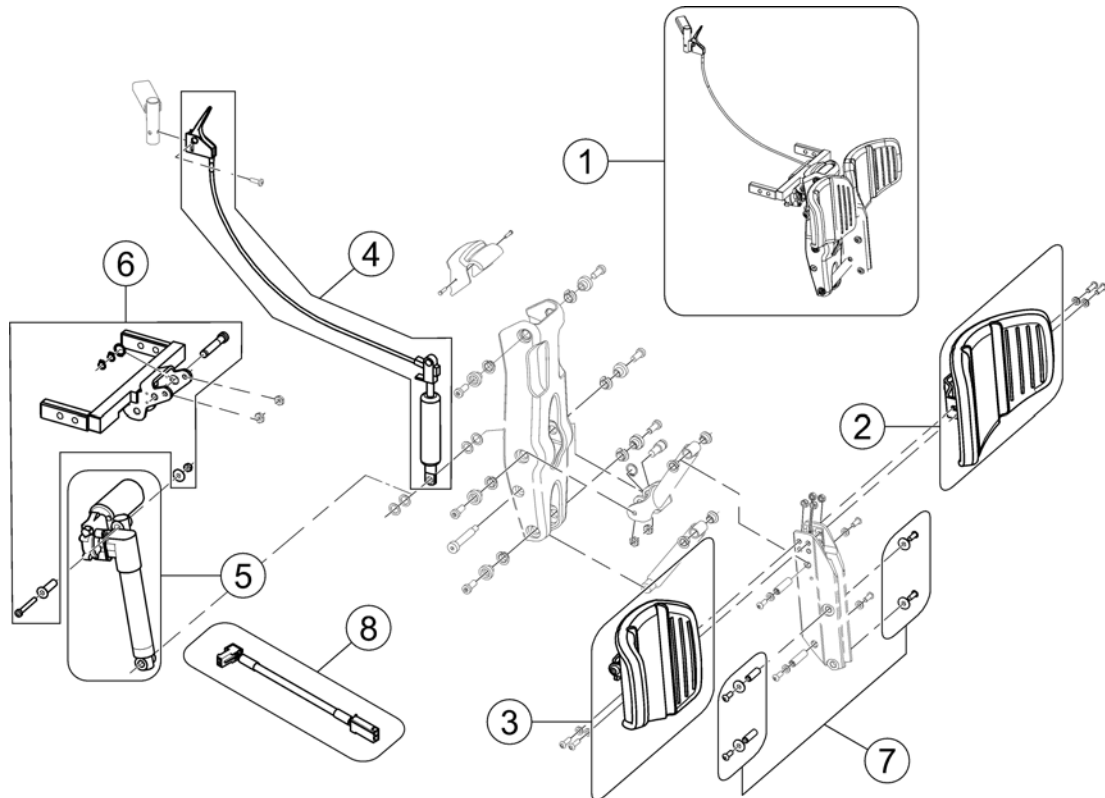

Pos.	Part number	Description	Valid from → until	Qty
1	SP1529388	Central Legrest, electric, complete		1
2	1436076	Calf Plate with holder LH		1
3	1440594	Calf Plate with holder RH		1
4	1440593	Gas Pressure Spring		1
5	SP1440600	Actuator el. Central Legrest 90°		1
6	SP1574298	Bracket for el. Central Legrest		1
7	SP1576334	Mounting Hardware Arm 2-E		1
8	SP1579552	Adapter Cable, Centr. Legrest (GGB2)		1

Armrests

SPCSTORM440.iss



Pos.	Part number	Description	Valid from → until	Qty
1	-	Armrest for Standard Seat		1
2	-	Armrests for EasyAdapt Seat		1
3	-	Armrests for Modulite Seat		1
4	-	Armrests for Ultra Low Maxx Seat		1
5	-	Armrest for Recaro Seat		1
6	-	Armrests for Optimist Seat		1

Armrest for Standard Seat


1573232.iss



Pos.	Part number	Description	Valid from → until	Qty
1	-	Fixed Armrest, removable [0635/0636]		1

Fixed Armrest, removable


1421692X.dwg

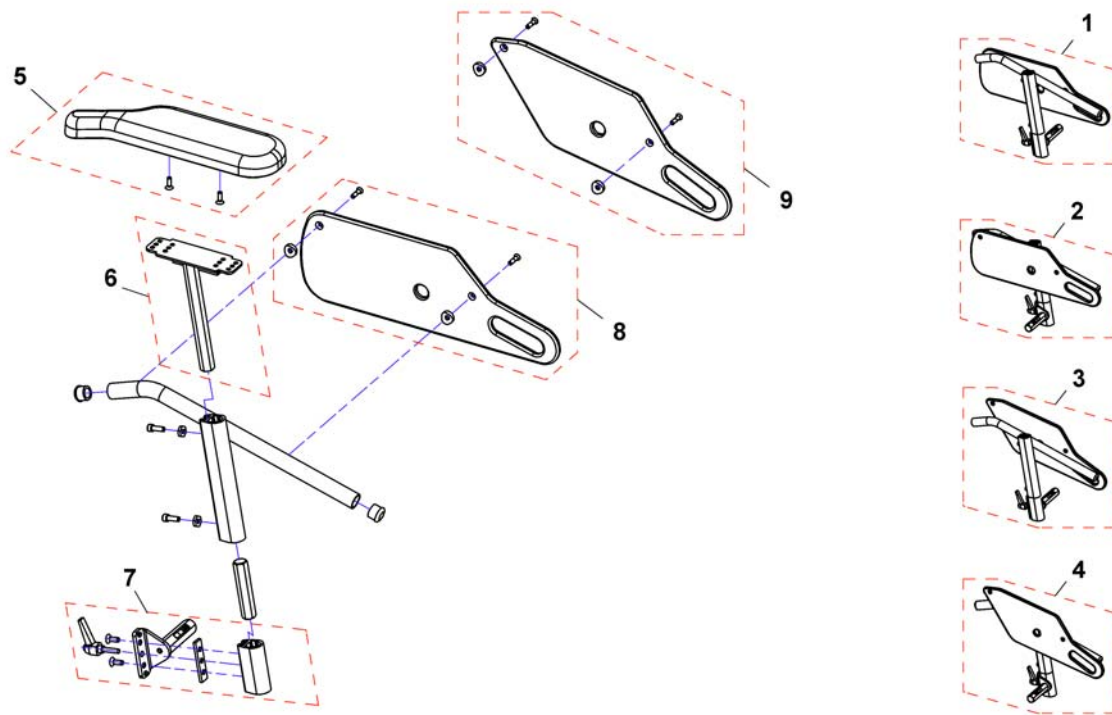


Pos.	Part number	Description	Valid from → until	Qty
1	SP1421692	Armrest, height max. 340mm, RH, Seatdepth 460mm & 510mm		1
1	SP1421693	Armrest, height max. 380mm, RH, Seatdepth 460mm & 510mm		1
2	SP1421694	Armrest, height max. 340mm, LH, Seatdepth 460mm & 510mm		1
2	SP1421695	Armrest, height max. 380mm, LH, Seatdepth 460mm & 510mm		1
3	SP1436533	Armrest, height max. 340mm, RH, Seatdepth 410mm		1
3	SP1436535	Armrest, height max. 380mm, RH, Seatdepth 410mm		1
4	SP1436534	Armrest, height max. 340mm, LH, Seatdepth 410mm		1
4	SP1436536	Armrest, height max. 380mm, LH, Seatdepth 410mm		1
5	SP1449043	Armpad, Standard (325mm), RH		1
5	SP1443004	Armpad, Standard (325mm), LH		1
5	SP1528845	Armpad, Extended (375mm), RH		1
5	SP1528846	Armpad, Extended (375mm), LH		1
6	SP1510856	Armpad holder RH <i>For chairs bevor 01/01/09 also order a new armpad.</i>		1
6	SP1510857	Armpad holder LH <i>For chairs bevor 01/01/09 also order a new armpad.</i>		1
7	SP1423223	Side part receptacle RH, cpl.		1
7	SP1423225	Side part receptacle LH, cpl.		1
8	SP1420471	Skirtguard, Seatdepth 460 & 510mm		1
9	SP1428078	Skirtguard, seat depth 410mm		1

Armrests for EasyAdapt Seat

1573233.iss

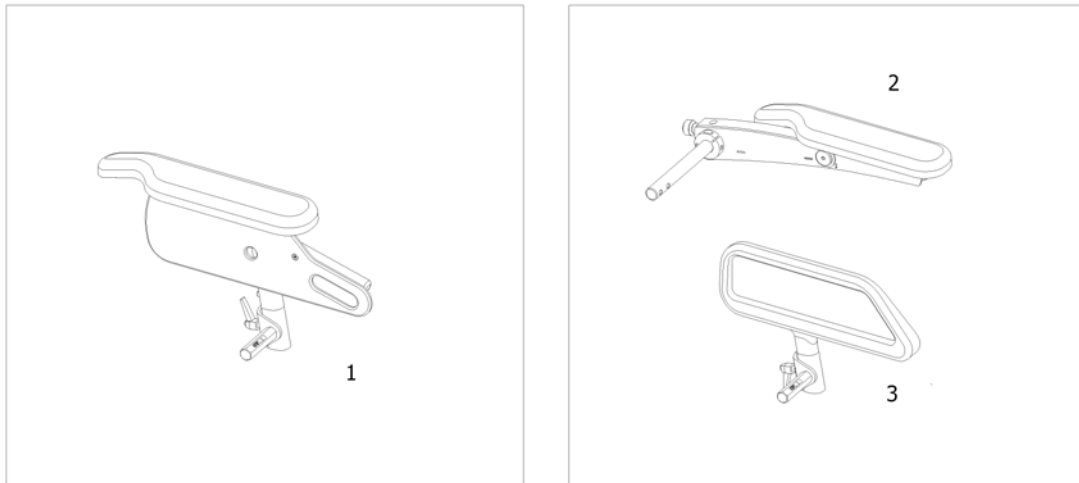


Pos.	Part number	Description	Valid from → until	Qty
1	-	Fixed Armrest, removable [0635/0636]		1
2	-	Following armrests [0620]		1
3	-	Skirtguard for following armrest [0623]		1

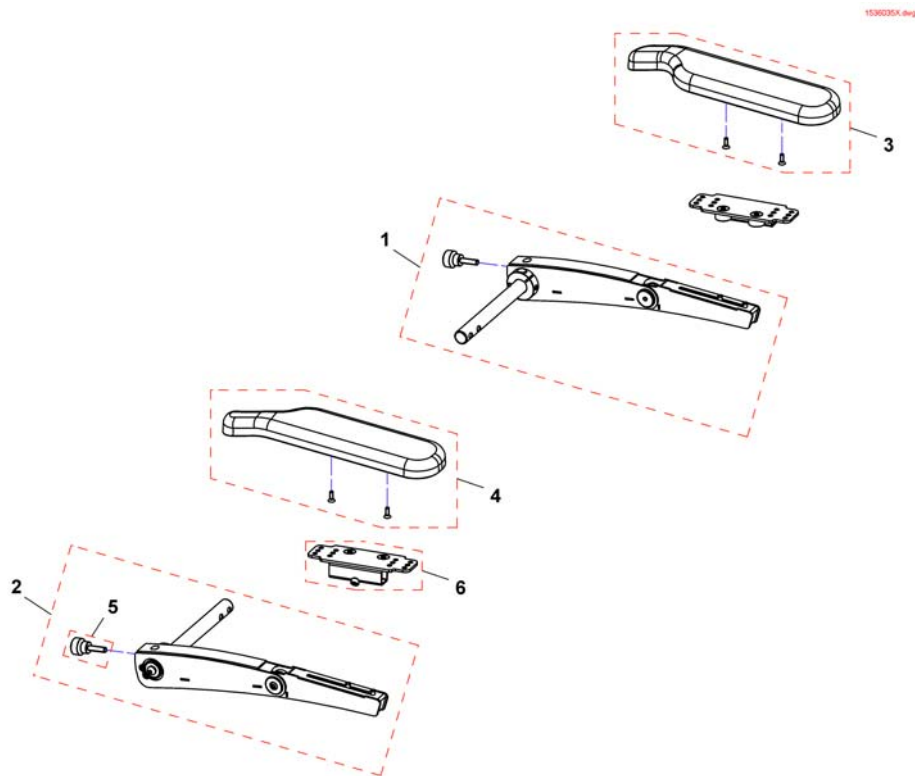


Pos.	Part number	Description	Valid from → until	Qty
1	SP1536036	Following armrest, short, LH, cpl.		1
1	SP1536100	Following armrest, long, LH, cpl.		1
2	SP1536035	Following armrest, short, RH, cpl.		1
2	SP1536099	Following armrest, long, RH, cpl.		1
3	SP1443004	Armpad, Standard (325mm), LH		1
3	SP1528846	Armpad, Extended (375mm), LH		1
4	SP1449043	Armpad, Standard (325mm), RH		1
4	SP1528845	Armpad, Extended (375mm), RH		1
5	SP1535504	Knurled screw M8x30		1
6	SP1535244	Armpad holder		1

Skirtguard for following armrest

1425860X.dwg

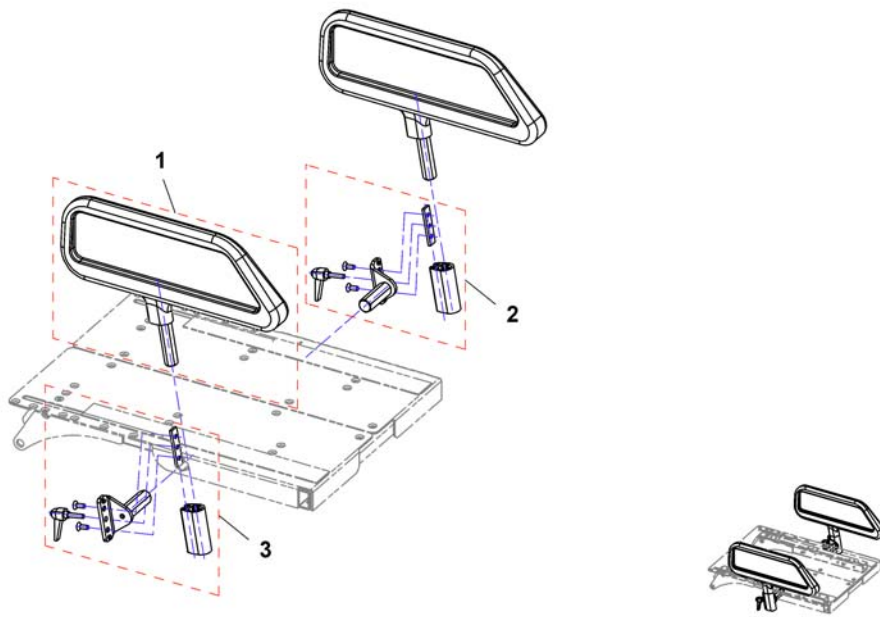


Pos.	Part number	Description	Valid from → until	Qty
1	SP1426303	Skirtguard		1
2	SP1442549	Bracket for Armrest / Skirtguard LH		1
3	SP1442550	Bracket for Armrest / Skirtguard RH		1

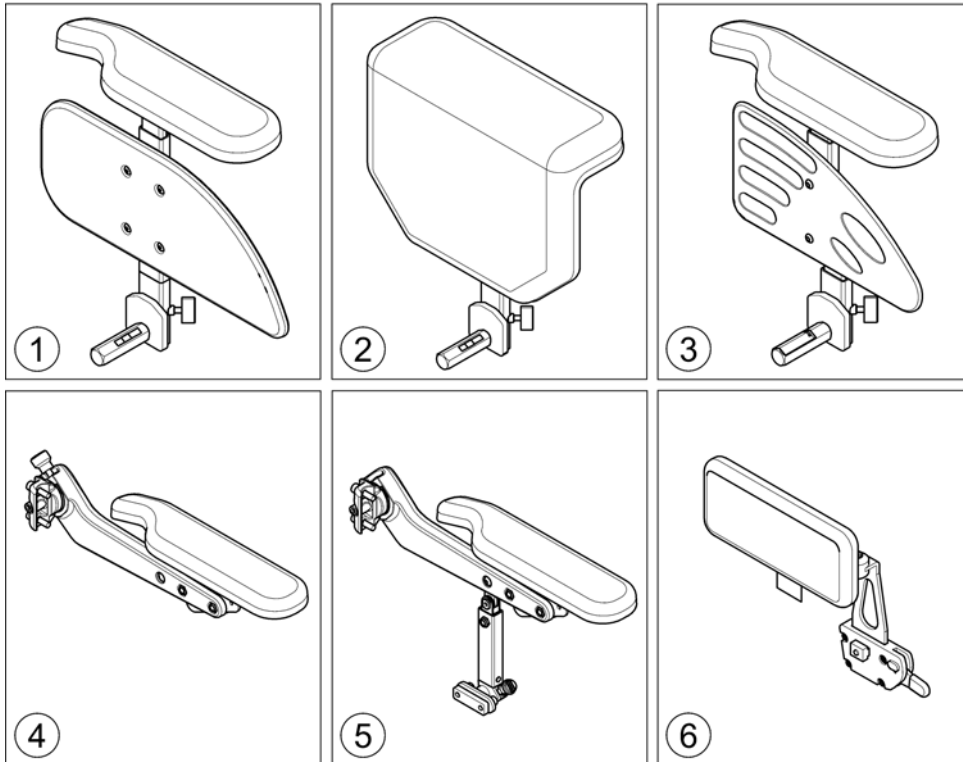

Pos.	Part number	Description	Valid from → until	Qty
1	-	T-Armrest [2191+2195/2196/2197]		1
2	-	Padded Armrest [2198]		1
3	-	Inrigged Armrest [2183+2195/2196/2197/2199]		0
4	-	Flip-Up Armrest [2193+2195/2196/2197/2199]		0
5	-	Following Armrest [2192/2194+2195/2196/2197/2199]		0
6	-	Hip Support [2184/2185/2186/2187/2188/2189]		0

T-Armrest


1572570 iso

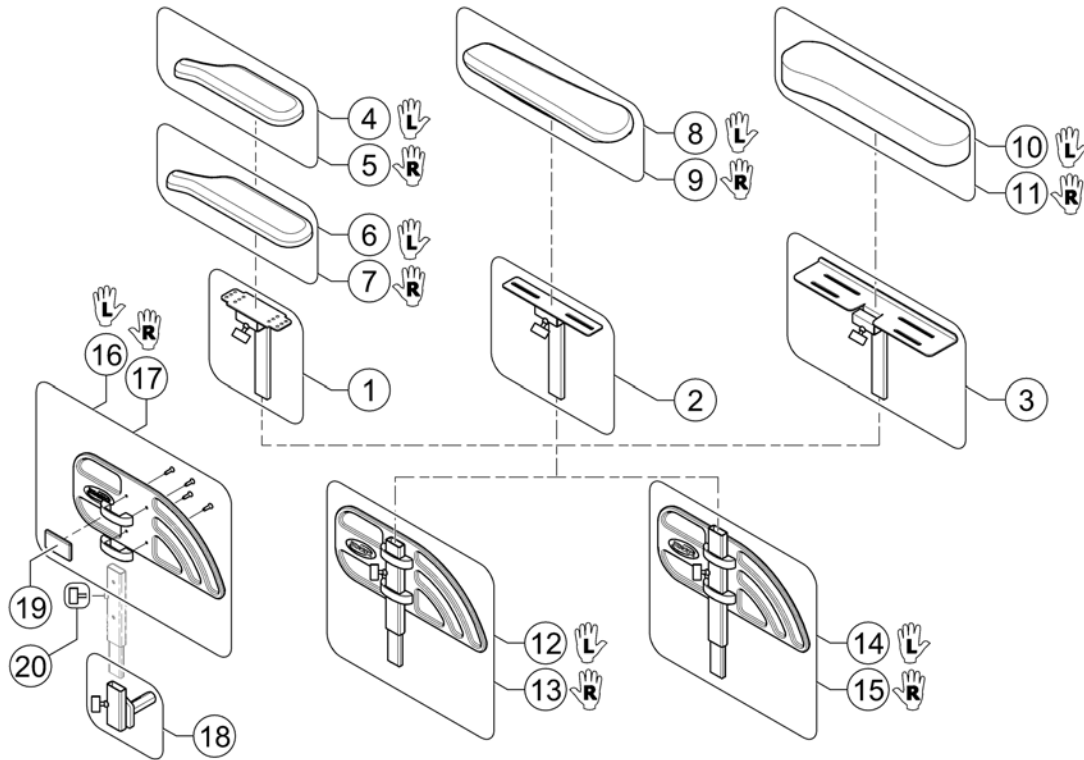

Pos.	Part number	Description	Valid from → until	Qty
1	SP1567750	Armrest Bracket for Standard & Extended Armpad		1
2	SP1570319	Armrest Bracket for Ergonomic Armpad		1
3	SP1585088	Armrest Bracket for Soft Armpad		1
4	SP1443004	Armpad, Standard (325mm), LH		1
5	SP1449043	Armpad, Standard (325mm), RH		1
6	SP1528846	Armpad, Extended (375mm), LH		1
7	SP1528845	Armpad, Extended (375mm), RH		1
8	SP1571589	Armpad, Ergonomic, LH		1
9	SP1571588	Armpad, Ergonomic, RH		1
10	SP1585068	Armpad, Soft, LH		1
11	SP1585069	Armpad, Soft, RH		1
12	SP1588374	Siderest LH, cpl. w/o brackets		1
13	SP1588375	Siderest RH, cpl. w/o brackets		1
14	SP1601882	Siderest (+50mm height) - LH, cpl. without brackets		1
15	SP1601883	Siderest (+50mm height) - RH, cpl. without brackets		1
16	SP1572940	Skirt Guard LH		1
17	SP1572948	Skirt Guard RH		1
18	SP1567749	Siderest Bracket - LH/RH		1
19	SPP31298	Reflector, yellow		1
20	SPL10024	Wing knobs M6x16		1


Padded Armrest

1573214 Rev

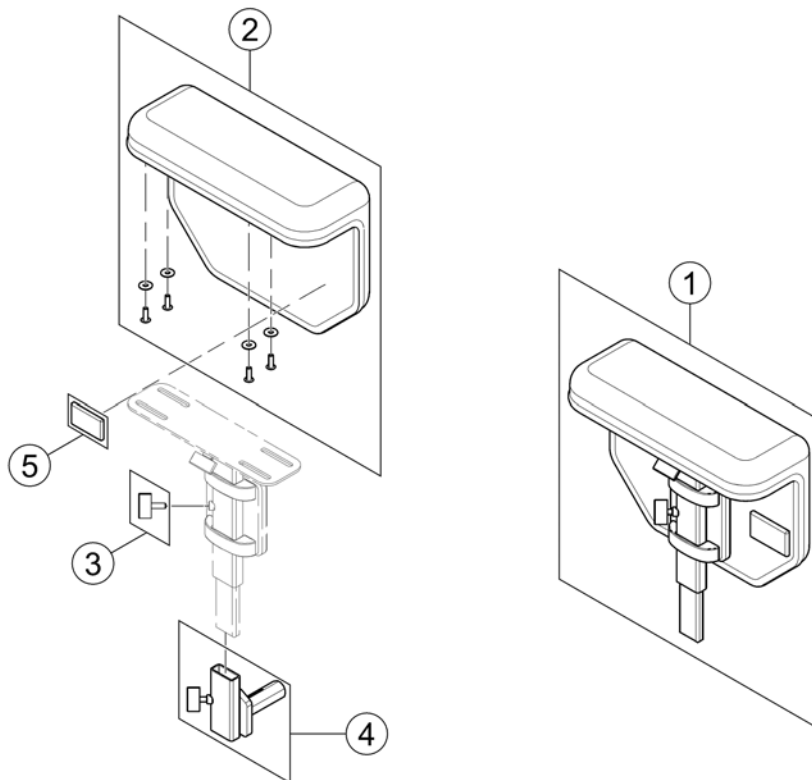

Pos.	Part number	Description	Valid from → until	Qty
1	SP1571987	Padded Armrest RH/LH, cpl.		1
2	SP1571962	Armpad, padded - RH/LH with integrated cloth guard		1
3	SPL10024	Wing knobs M6x16		1
4	SP1567749	Siderest Bracket - LH/RH		1
5	SPP31298	Reflector, yellow		1


Inrigged Armrest

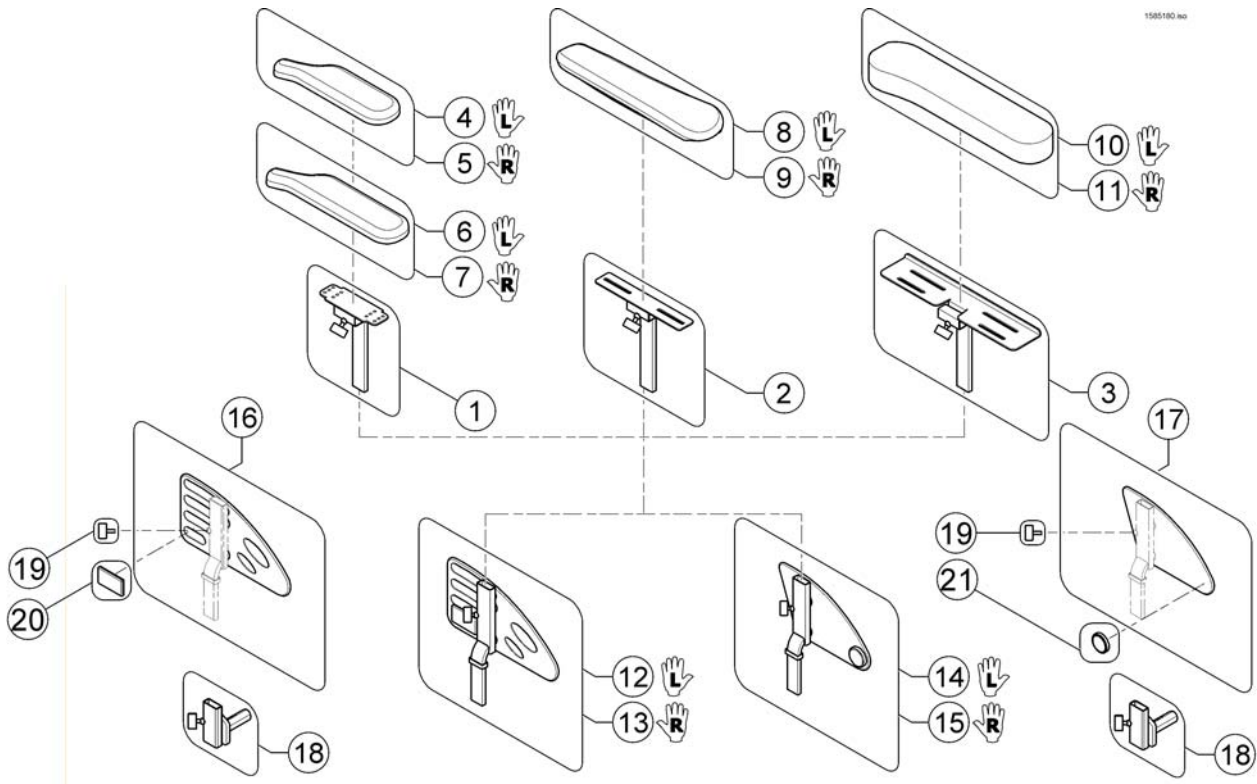

Pos.	Part number	Description	Valid from → until	Qty
1	SP1567750	Armrest Bracket for Standard & Extended Armpad		1
2	SP1570319	Armrest Bracket for Ergonomic Armpad		1
3	SP1585088	Armrest Bracket for Soft Armpad		1
4	SP1443004	Armpad, Standard (325mm), LH		1
5	SP1449043	Armpad, Standard (325mm), RH		1
6	SP1528846	Armpad, Extended (375mm), LH		1
7	SP1528845	Armpad, Extended (375mm), RH		1
8	SP1571589	Armpad, Ergonomic, LH		1
9	SP1571588	Armpad, Ergonomic, RH		1
10	SP1585068	Armpad, Soft, LH		1
11	SP1585069	Armpad, Soft, RH		1
12	SP1585037	Siderest inrigged LH		1
13	SP1585038	Siderest inrigged RH		1
14	SP1605654	Siderest inrigged LH (SD410mm)		1
15	SP1605653	Siderest inrigged RH (SD410mm)		1
16	SP1578119	Skirt Guard LH/RH		1
17	SP1603118	Skirt Guard LH/RH (SD410mm)		1
18	SP1567749	Siderest Bracket - LH/RH		1
19	SPL10024	Wing knobs M6x16		1
20	SPP31298	Reflector, yellow		1
21	SP1582989	Reflector yellow D=43/38mm		1


Flip-Up Armrest

1580181 Rev

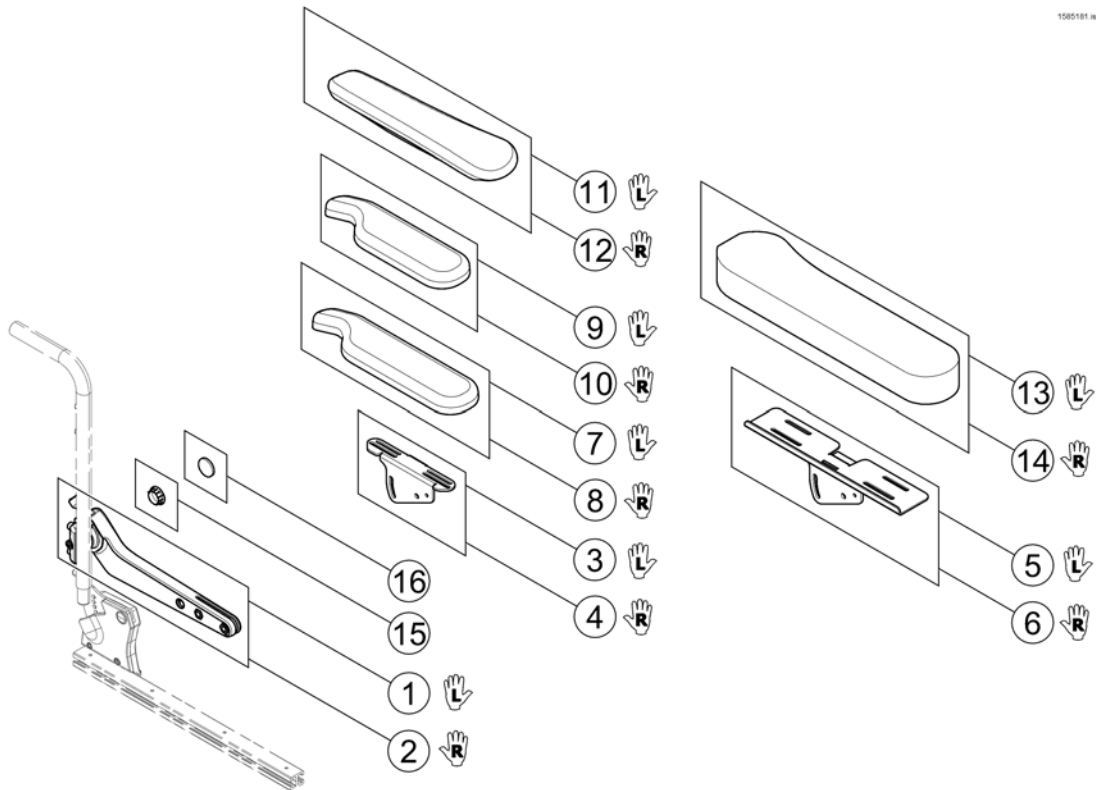

Pos.	Part number	Description	Valid from → until	Qty
-	-	Retrofitting the armrests on chairs built before 15-03-27 also requires new Pos. A & B and Sling Back with respective cut-outs! !!!		0
1	SP1582668	Flip-Up Armrest, LH	2015-01-19 →	1
2	SP1582644	Flip-Up Armrest, RH	2015-01-19 →	1
3	SP1585412	Ampad Support (Standard, Extended, Ergonomic), LH		1
4	SP1585413	Ampad Support (Standard, Extended, Ergonomic), RH		1
5	SP1585414	Ampad Support (Soft), LH		1
6	SP1585415	Ampad Support (Soft), RH		1
7	SP1528846	Ampad, Extended (375mm), LH		1
8	SP1528845	Ampad, Extended (375mm), RH		1
9	SP1443004	Ampad, Standard (325mm), LH		1
10	SP1449043	Ampad, Standard (325mm), RH		1
11	SP1571589	Ampad, Ergonomic, LH		1
12	SP1571588	Ampad, Ergonomic, RH		1
13	SP1585068	Ampad, Soft, LH		1
14	SP1585069	Ampad, Soft, RH		1
15	SP1590599	Cap		1
16	SP1582989	Reflector yellow D=43/38mm		1

Following Armrest

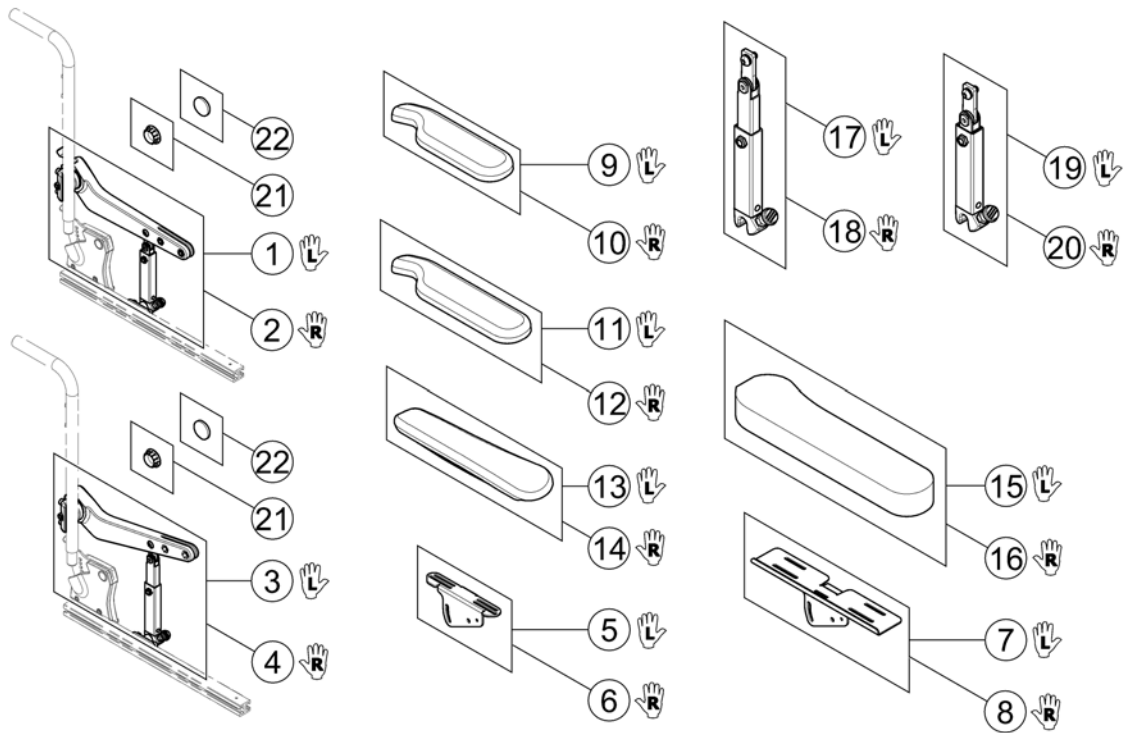


1585182 No

Pos.	Part number	Description	Valid from → until	Qty
-	-	Retrofitting the armrests on chairs built before 15-03-27 also requires new Pos. A & B and Sling Back with respective cut-outs! !!!		0
1	SP1585263	Armrest LH cpl. - Short Support	2015-01-19 →	1
2	SP1585264	Armrest RH cpl. - Short Support	2015-01-19 →	1
3	SP1585267	Armrest LH cpl. - Long Support	2015-01-19 →	1
4	SP1585268	Armrest RH cpl. - Long Support	2015-01-19 →	1
5	SP1585412	Ampad Support (Standard, Extended, Ergonomic), LH		1
6	SP1585413	Ampad Support (Standard, Extended, Ergonomic), RH		1
7	SP1585414	Ampad Support (Soft), LH		1
8	SP1585415	Ampad Support (Soft), RH		1
9	SP1443004	Ampad, Standard (325mm), LH		1
10	SP1449043	Ampad, Standard (325mm), RH		1
11	SP1528846	Ampad, Extended (375mm), LH		1
12	SP1528845	Ampad, Extended (375mm), RH		1
13	SP1571589	Ampad, Ergonomic, LH		1
14	SP1571588	Ampad, Ergonomic, RH		1
15	SP1585068	Ampad, Soft, LH		1
16	SP1585069	Ampad, Soft, RH		1
17	SP1583156	Amrest Support, long, LH		1
18	SP1583155	Amrest Support, long, RH		1
19	SP1583154	Amrest Support, short, LH		1
20	SP1582639	Amrest Support, short, RH		1
21	SP1590599	Cap		1
22	SP1582989	Reflector yellow D=43/38mm		1

Hip Support

1585183 Rev

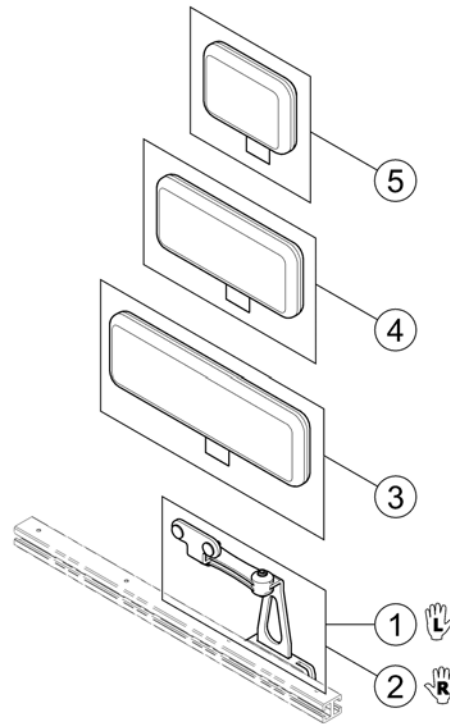


Pos.	Part number	Description	Valid from → until	Qty
1	SP1584411	Hip Support, Quick Release bracket, LH		1
2	SP1584410	Hip Support, Quick Release bracket, RH		1
3	SP1584414	Hip Support Pad, large (330x100 mm)		1
4	SP1584413	Hip Support Pad, medium (230x100 mm)		1
5	SP1584412	Hip Support Pad, small (140x100 mm)		1

Armrests for Ultra Low Maxx Seat

Pos.	Part number	Description	Valid from → until	Qty
1	-	Recline Flip-Up Armrest		0
2	-	Cantilever Flip-Up Armrest		0
3	-	Armpads, Desk & Full		0
4	-	Armpad, Ergonomic		0
5	-	Elbow Block		0
6	-	MACES Multi Axis Armpad Holder		0
7	-	Hip Support		0


1602129 Rev

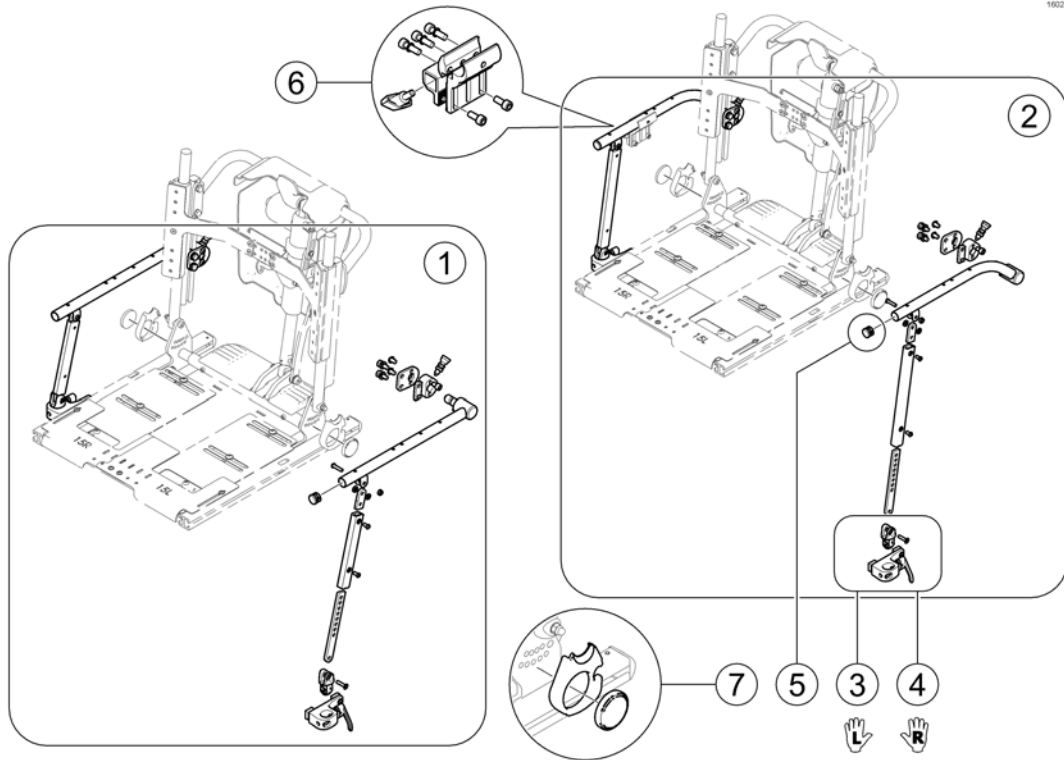


Pos.	Part number	Description	Valid from → until	Qty
1	SPTRC1933	Reclining Arm Flip-Back 240-330mm		1
2	SPTRC1934	Reclining Arm Flip-Back 330-405mm		1
3	SPTRC2135	Flip Back Lever with connector LH		1
4	SPTRC2134	Flip Back Lever with connector RH		1
5	SPTRM0130	Armrest-pipe cap		1
6	SPTRC2060	Clamp with square tube adapter		1
7	SP1602015	Reflector Kit ULM - LH		1
7	SP1602016	Reflector Kit ULM - RH		1

Cantilever Flip-Up Armrest

1602155.iso

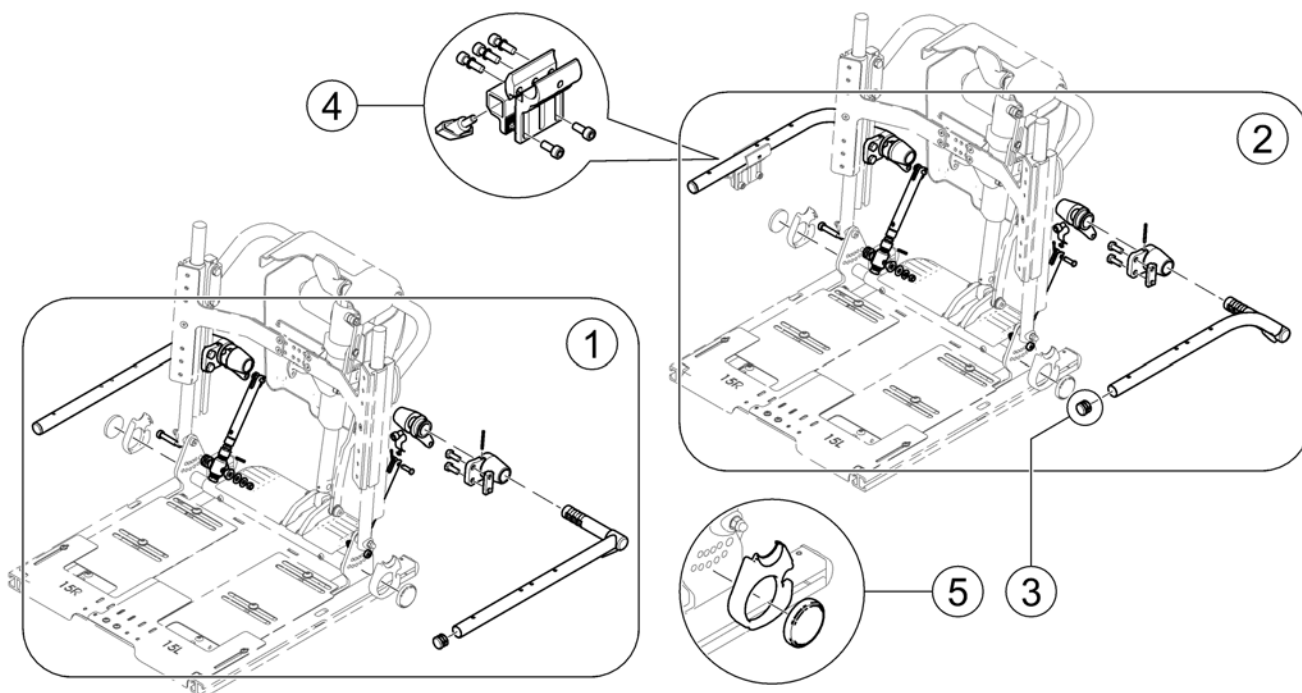


Pos.	Part number	Description	Valid from → until	Qty
1	SPTRC1936	Cantilever Flip-Back Desk 240-330mm		1
2	SPTRC1937	Cantilever Flip-Back Desk 320-405mm		1
3	SPTRM0130	Armrest-pipe cap		1
4	SPTRC2060	Clamp with square tube adapter		1
5	SP1602015	Reflector Kit ULM - LH		1
5	SP1602016	Reflector Kit ULM - RH		1

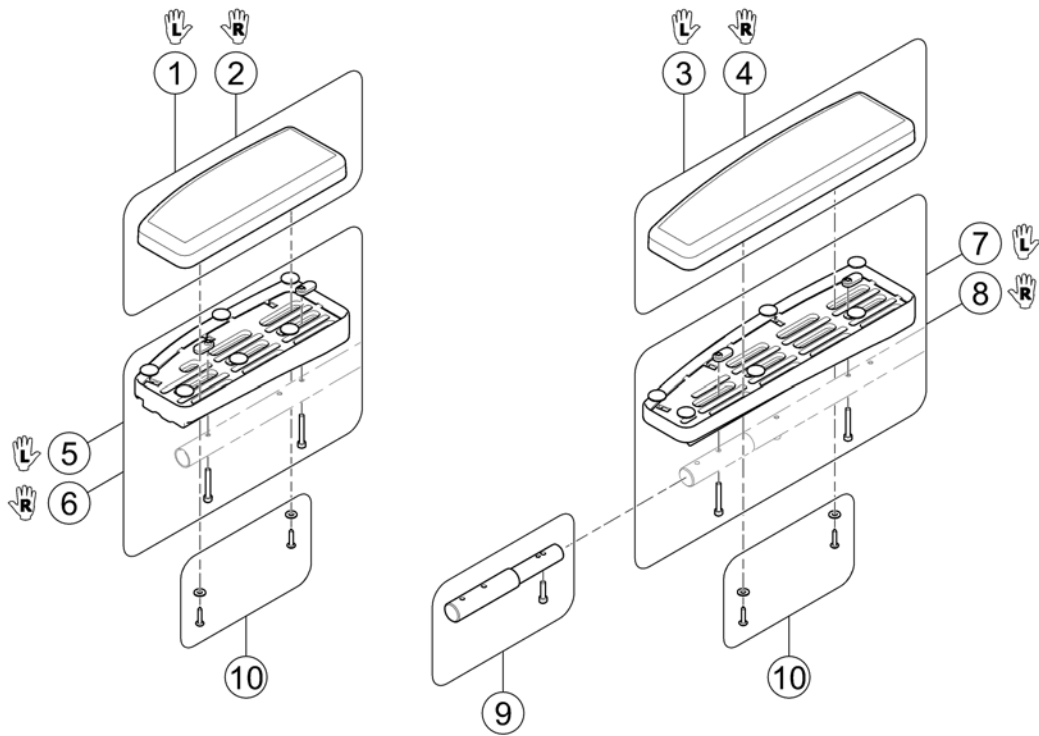

Pos.	Part number	Description	Valid from → until	Qty
1	SPTRX8038	Armpad, Desk Amrest, LH		1
2	SPTRX8037	Armpad, Desk Amrest, RH		1
3	SPTRX8036	Armpad, Full Amrest, LH		1
4	SPTRX8035	Armpad, Full Amrest, RH		1
5	SPTRC2133	Desk Armrest;Body LH with mountig hardware		1
6	SPTRC2132	Desk Armrest;Body RH with mountig hardware		1
7	SPTRC2131	Full Armrest;Body LH with mountig hardware		1
8	SPTRC2130	Full Armrest;Body RH with mountig hardware		1
9	SPTRC1935	Desk to Full Armrest Extension Kit <i>Needed for Pos. 7 & 8!</i>		1
10	SP1602292	Screw set for armpad fixation <i>[1494]</i>		1

Armpad, Ergonomic


1588506 Rev

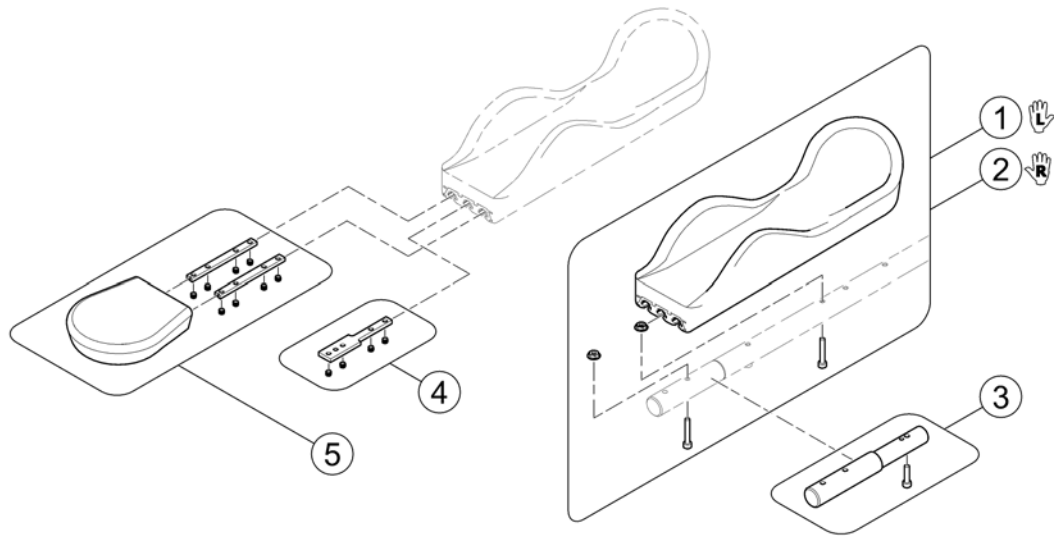

Pos.	Part number	Description	Valid from → until	Qty
1	SPTRC2100-C02	Ergonomic Arm Trough LH		1
2	SPTRC2100-C01	Ergonomic Arm Trough RH		1
3	SPTRC1935	Desk to Full Armrest Extension Kit <i>Needed for Pos. 1 & 2!</i>		1
4	SPTRC2102	Mounting Hardware for Otto Bock		1
5	SPTRC2101	Flat Hand Pad		1

Elbow Block


1603428.iss

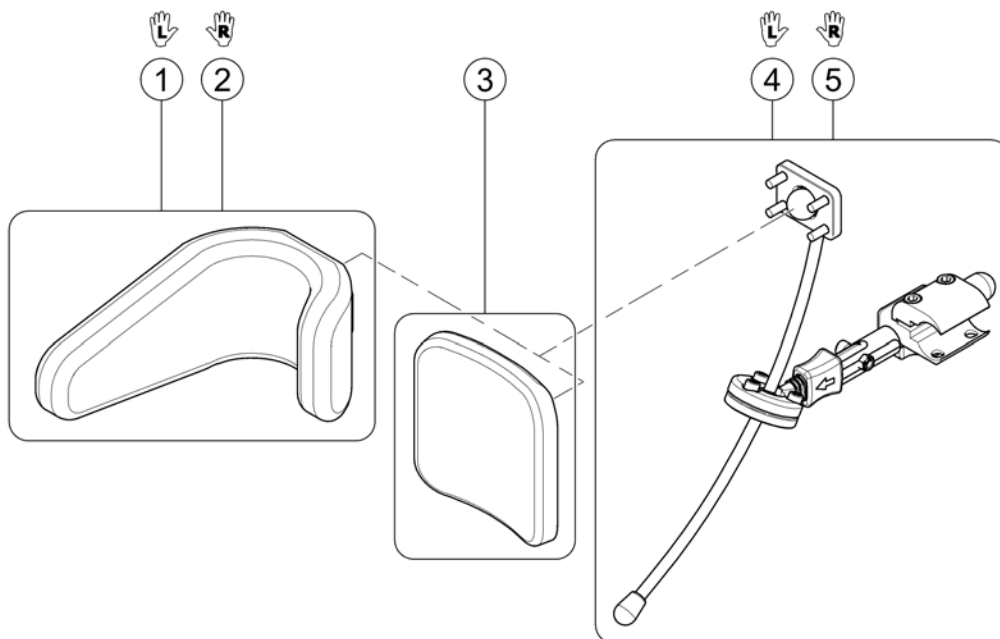

Pos.	Part number	Description	Valid from → until	Qty
1	SPTRM1059	Elbow Block pad 3x10 LH		1
2	SPTRM1060	Elbow Block pad 3x10 RH		1
3	SPTRM1058	Elbow Block pad 4x4		1
4	SPTRM1055	Elbow Block Hardware, cpl. LH		1
5	SPTRM1056	Elbow Block Hardware, cpl. RH		1

MACES Multi Axis Armpad Holder

1588515 Rev

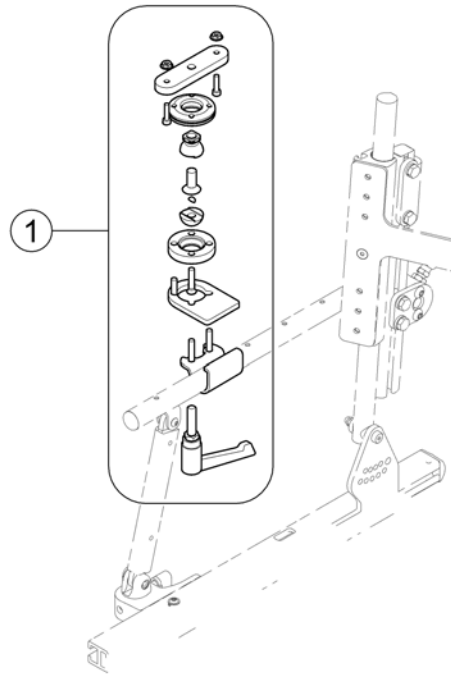


Pos.	Part number	Description	Valid from → until	Qty
1	SPTRC1938	MACES Armpad Holder Kit, Multi Axis		1



Pos.	Part number	Description	Valid from → until	Qty
1	-	Fixed Armrest, removable [0636]		1


Armrests for Optimist Seat

1573236.no

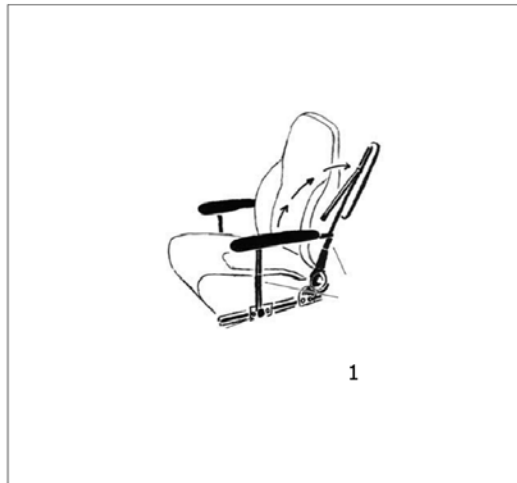

Pos.	Part number	Description	Valid from → until	Qty
1	-	Armrests Arne Jensen for Optimist Seat [0651]		1


Armrests Arne Jensen for Optimist Seat

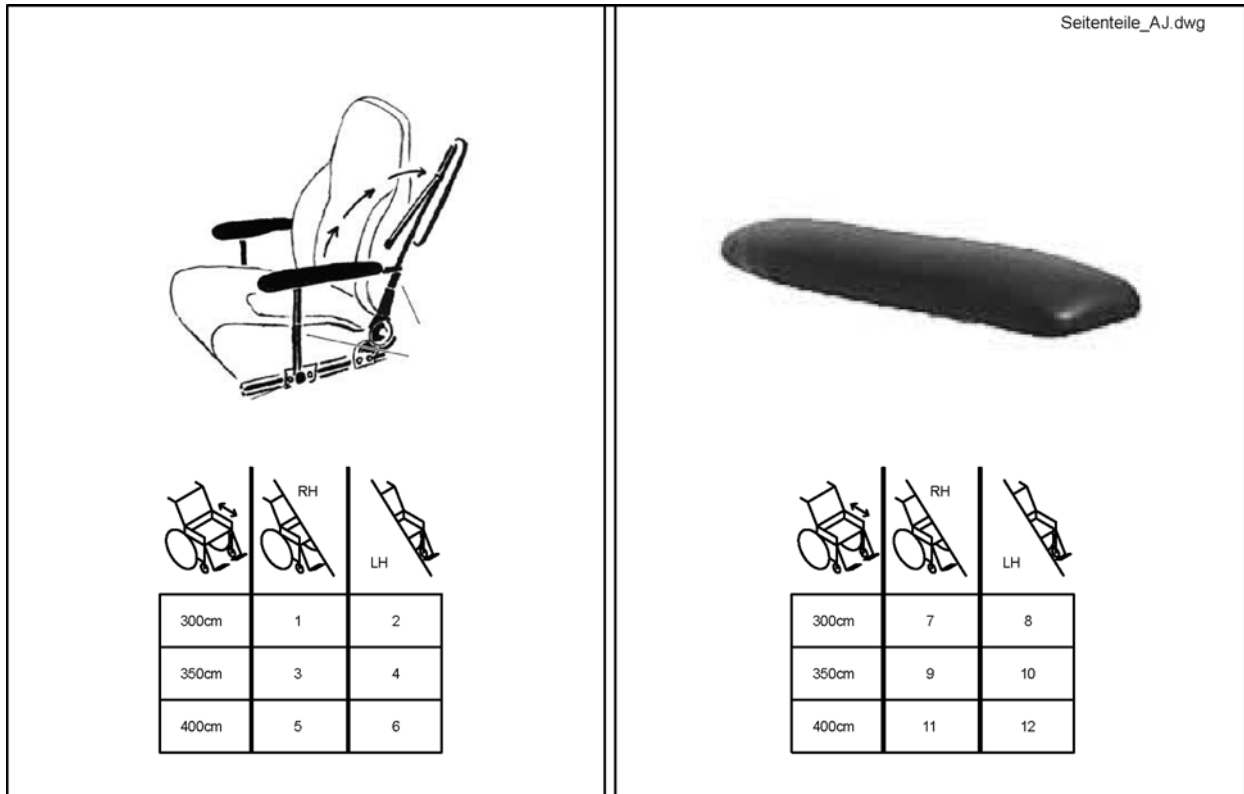

Pos.	Part number	Description	Valid from → until	Qty
1	1419657	Armrest Optimist 30cm RH		1
2	1419658	Armrest Optimist 30cm LH		1
3	1419659	Armrest Optimist 35cm RH		1
4	1419660	Armrest Optimist 35cm LH		1
5	1419661	Armrest Optimist 40cm RH		1
6	1419662	Armrest Optimist 40cm LH		1
7	1419647	Armrest Pad Optimist 30cm RH		1
8	1419648	Armrest Pad Optimist 30cm LH		1
9	5314011	Armrest Pad Optimist 35cm RH		1
10	5314010S	Armrest Pad Optimist 35cm LH		1
11	5314011L	Armrest Pad Optimist 40cm RH		1
12	5314010	Armrest Pad Optimist 40cm LH		1

Front Fork


152890X.kim

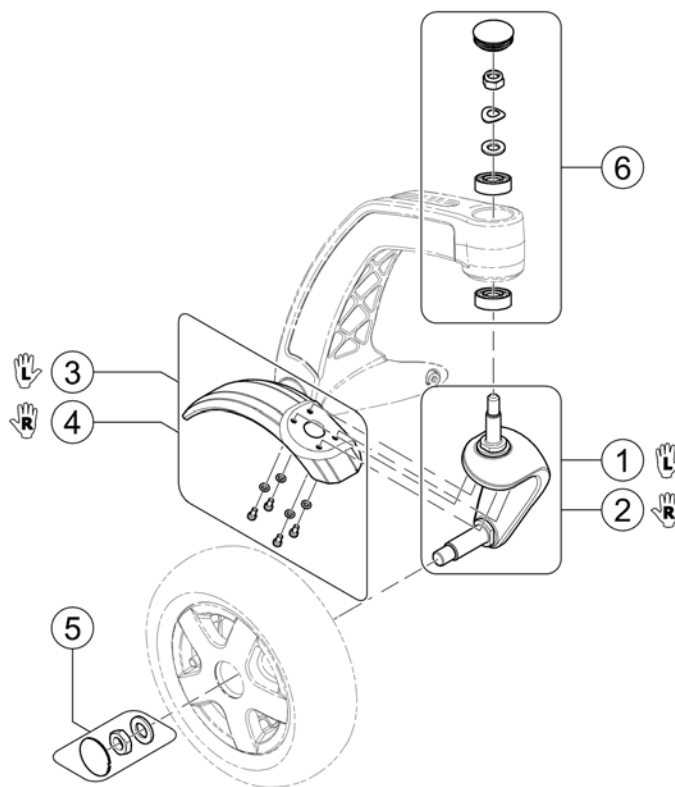

Pos.	Part number	Description	Valid from → until	Qty
1	SP1588447	Front Fork, cpl. - LH <i>We recommend to exchange the front forks by pairs, because of new design. - replaces SP1552920</i>	2015-07-30 →	1
2	SP1588448	Front Fork, cpl. - RH <i>We recommend to exchange the front forks by pairs, because of new design. - replaces SP1552926</i>	2015-07-30 →	1
3	SP1530553	Fender for Castor, cpl. - LH		1
4	SP1530530	Fender for Castor, cpl. - RH		1
5	SP1530554	Wheel Fixation Kit		1
6	SP1530529	Bearing-Kit		1

Front Wheels

1419256.dwg

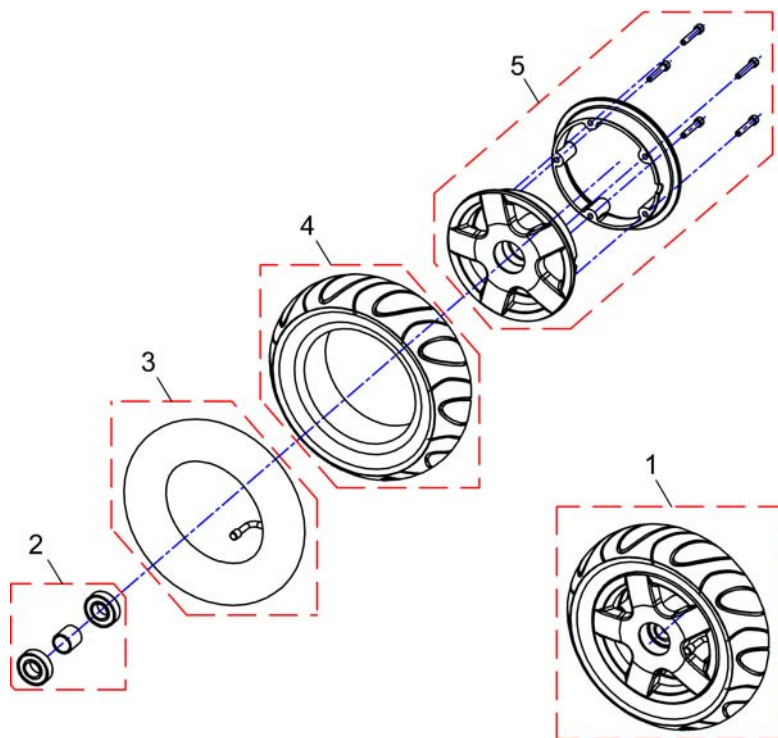


Pos.	Part number	Description	Valid from → until	Qty
1	SP1418531	WHEEL 3.00-6" AIR, GREY, CPL.		1
1	SP1422932	WHEEL 3.00-6" PUNCTURE-PROTECTED, GREY, CPL.		1
1	SP1430231	WHEEL 3.00-6" PUNCTURE-PROOF, SOLID INLAY, GREY, CPL.		1
2	SP1423182	WHEEL RIM BEARING SET 6", CPL.		1
3	1415175	TUBE 3.00 X 6 90 ° TR 87		1
3	SP1557438	TUBE 3.00 X 6 90 ° TR 87, PUNCTURE PROTECTED		1
4	1418532	COVER 3.00-6", GREY		1
4	1430229	OUTER TYRE 3.00-6 GREY PUNCTURE-PROOF SOLID INLAY		1
5	SP1423181	WHEEL RIM ASSEMBLY KIT 6", CPL. (for 10" rim)		1

Storm4

Pos.	Part number	Description	Valid from → until	Qty
-		Drive Wheel Trelleborg (8x3.00)		1
-		Drive Wheel 14" (3.00 - 8)		1


Drive Wheel Trelleborg (8x3.00)

1421832X.dwg

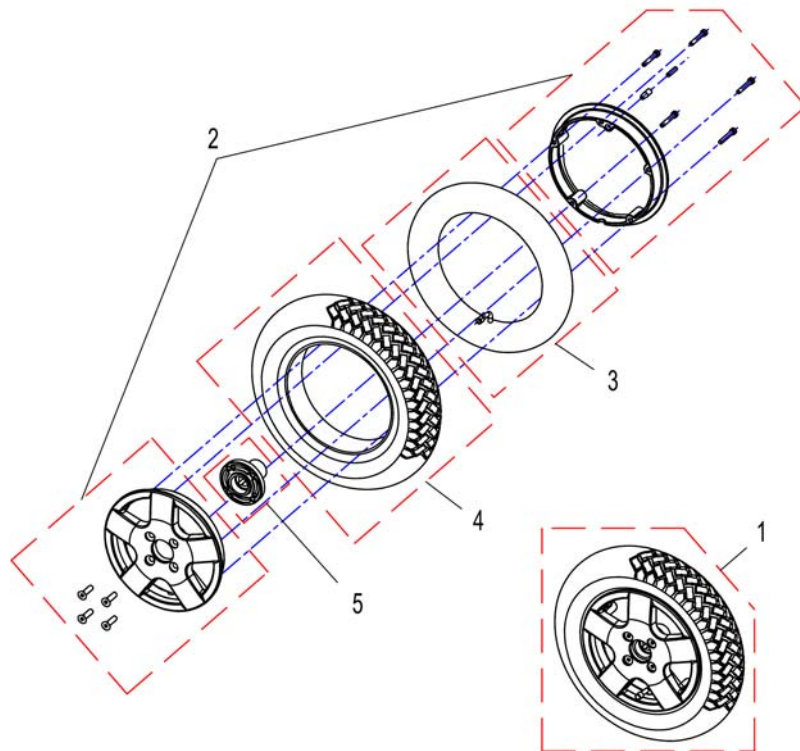

Pos.	Part number	Description	Valid from → until	Qty
1	SP1421800	Rear Wheel, Trelleborg, pneumatic (without hub)		1
1	SP1421832	Rear Wheel, Trelleborg, puncture proof (without hub)		1
2	SP1423184	Wheel Rim Assembly Kit 8", cpl.		1
3	M12016	Tube 3.00-8" with valve TR 87-90/90°		1
4	M11968	Tyre 3.00-8", grey, Trelleborg		1
5	1509054-0002	Wheel Hub, 4-Hole <i>Replaced by SP1541458</i>	→ 2011-01-04	1
5	SP1541458	Wheelhub (SSD), 4-Hole <i>Replaces 1509054-0002</i>	2011-01-05 →	1

Drive Wheel 14" (3.00 - 8)

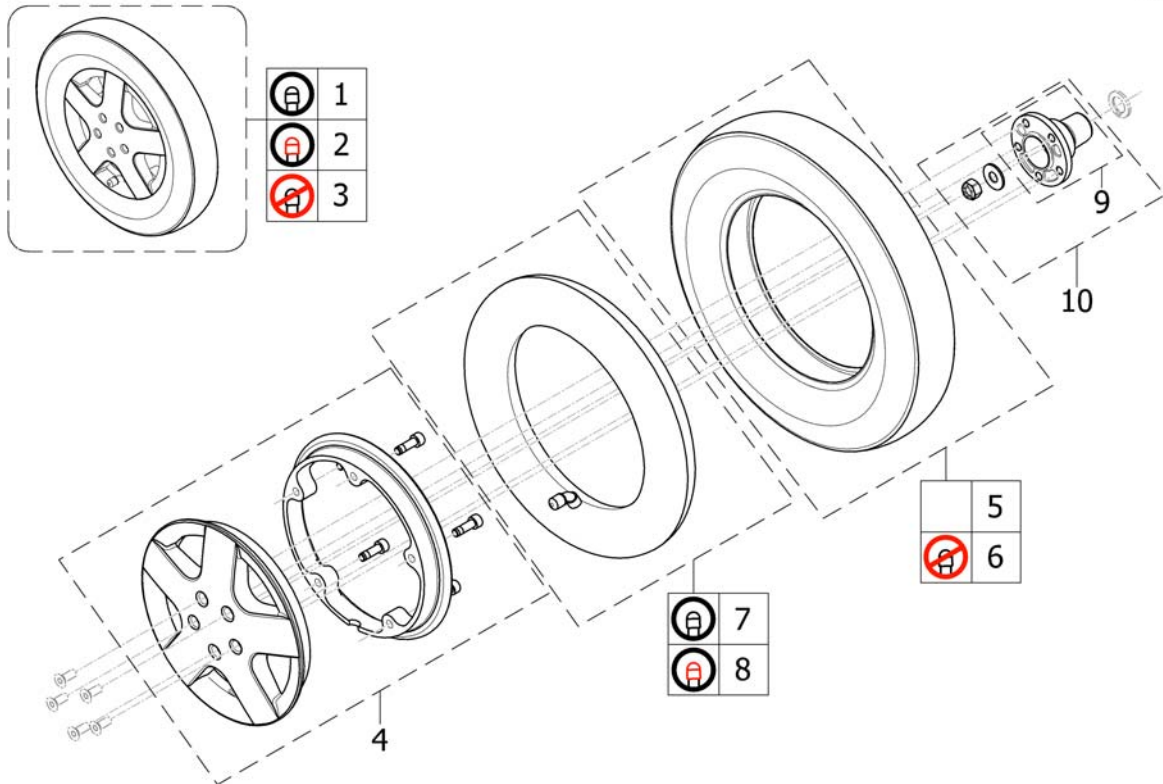


Pos.	Part number	Description	Valid from → until	Qty
1	SP1532414	Wheel 14 x 3.00" pneumatic, silver, complete		1
2	SP1532415	Wheel 14 x 3.00" puncture protected, silver, complete		1
3	SP1532416	Wheel 14 x 3.00" puncture proof, silver, complete		1
4	SP1541373	Rim Kit, complete		1
-	SP1522975	Bolt (5/16-18 x 7/8")		5
5	SP1522940	Tyre 14, pneumatic (3.00-8)		1
6	SP1141342	Tyre 14", puncture proof (3.00-8)		1
7	SP1039651	Tube 14"		1
8	SP1541374	Tube 14" puncture protected		1
9	SP1163728	Wheel hub 5-hole <i>05.01.2011 => - Replaces SP1536413</i>	2011-01-05 →	1
10	SP1536413	Wheelhub Kit, complete <i>=>05.01.2011 - Replaced by SP1163728</i>	→ 2011-01-04	1

Storm4 TTPlus

Pos.	Part number	Description	Valid from → until	Qty
-		Drive Wheels GGB		0

Drive Wheels GGB

1581908.iso

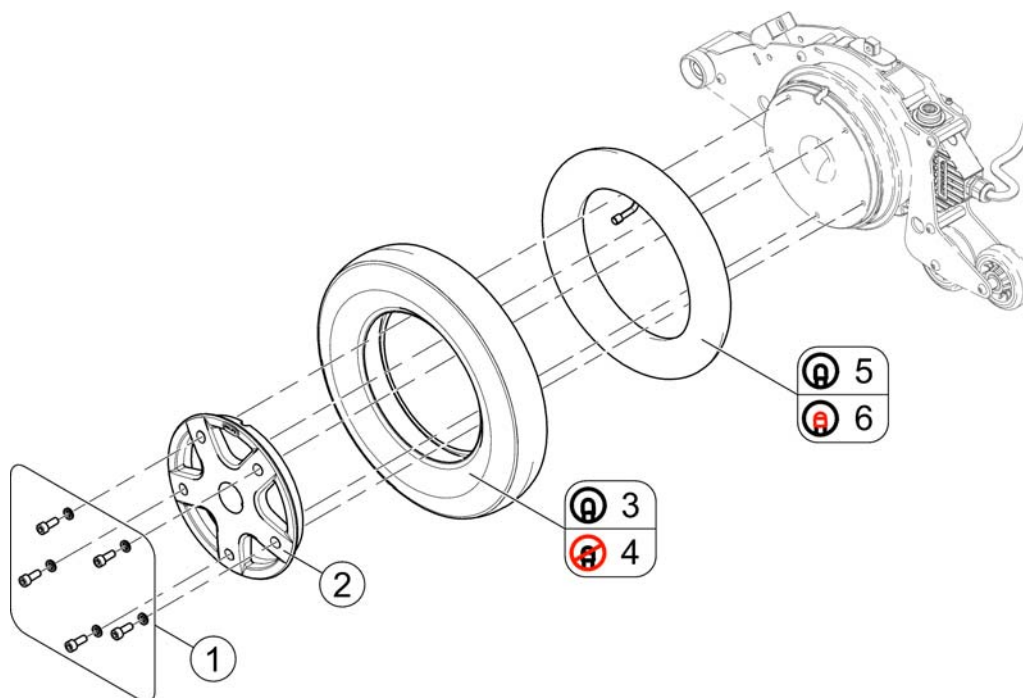


Pos.	Part number	Description	Valid from → until	Qty
1	SP1582235	Wheel mounting kit		1
2	SP1582236	Outer Wheel Rim 14" GGB2 incl. Sticker		1
3	SP1522940	Tyre 14, pneumatic (3.00-8)		1
4	SP1141342	Tyre 14", puncture proof (3.00-8)		1
5	SP1039651	Tube 14"		1
6	SP1541374	Tube 14" puncture protected		1

Storm4

Pos.	Part number	Description	Valid from → until	Qty
-		Propulsion => 04.01.11		0
-		Overview Propulsion SSD		0

Propulsion => 04.01.11


1528309X.iso

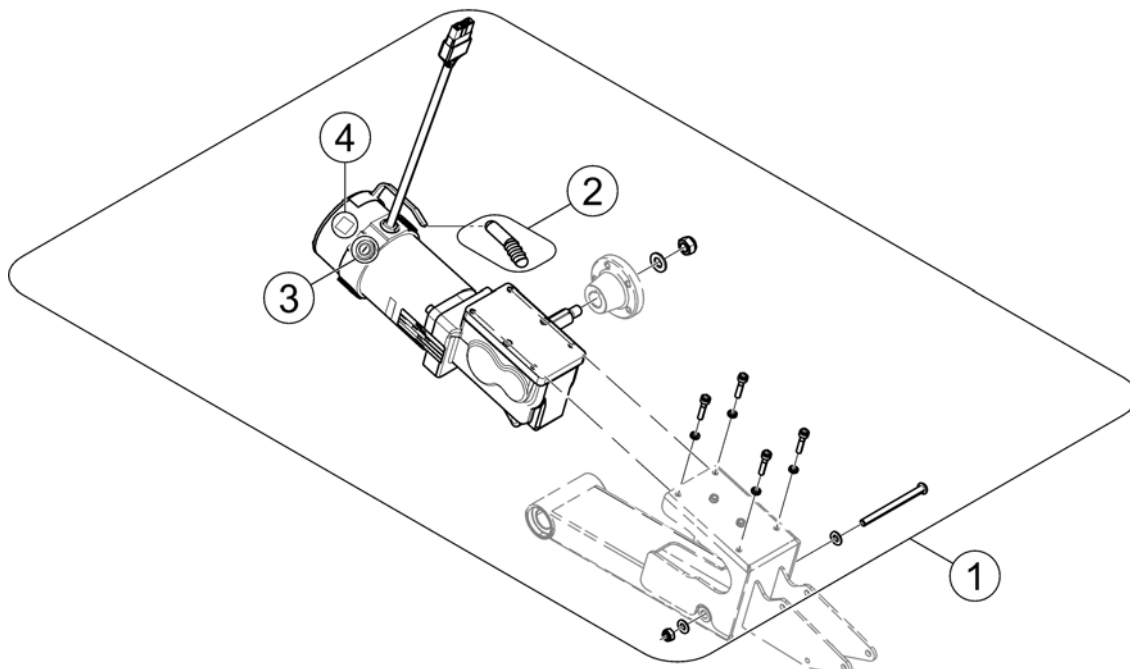

Pos.	Part number	Description	Valid from → until	Qty
1	SP1530578	Drivingmotor, 6km/h, cpl., LH => S/N 061122812867		1
1	SP1530577	Drivingmotor, 6km/h, cpl., RH => S/N 061122812867		1
1	SP1530580	Drivingmotor, 10km/h, cpl., LH => S/N 061122610150		1
1	SP1530579	Drivingmotor, 10km/h, cpl., RH => S/N 061122610150		1
2	SP1101725	Cap, Declutching Lever		1
3	SP1522967	Kit, brushes with caps, 4-pole (Set of 4)		1
4	SP1551712	Labelset "Push/Drive"		1

Overview Propulsion SSD

1163090X.iso

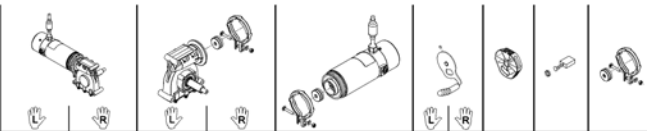

Bora/ SpectraXTR	6km/h	ACS/LINX /Shark	1	2	33	34	37	---	---	51	45
		VR2	3	4	33	34	43	---	---	51	45
		R-Net	5	6	33	34	41	---	---	51	45
	10km/h	ACS/LINX /Shark	7	8	31	32	36	---	---	51	45
		VR2	9	10	31	32	40	---	---	51	45
		R-Net	11	12	31	32	42	---	---	51	45
BoraHD/ Spectra XTR2HD	6km/h	ACS/LINX /Shark	54	55	33	34	39	---	---	51	46
	10km/h		56	57	31	32	38	---	---	51	46
	6km/h	R-Net	58	59	33	34	62	---	---	51	46
	10km/h		60	61	31	32	63	---	---	51	46
FDX	6km/h ^c	ACS/ Shark	13	14	33	34	37	52	53	---	45
	6km/h ^d		54	55	33	34	39	52	53	---	46
	10km/h		15	16	31	32	38	52	53	---	46
Kite	6km/h	ACS/LINX /Shark	17	18	33	34	37	---	---	51	45
	10km/h		19	20	31	32	36	---	---	51	45
Kite HD	6km/h	ACS/LINX /Shark	21	22	33	34	39	---	---	51	46
	10km/h		23	24	31	32	38	---	---	51	46
Storm4/ Storm4 X-plore	6km/h	ACS/ Shark	47	48	34	33	39	---	---	51	46
	10km/h		27	28	32	31	38	---	---	51	46
TDX-SP TDX-SP2	6km/h ^a	ACS/LINX /Shark	25	26	34	33	39	52	53	---	46
	8km/h ^a		29	30	34	33	38	52	53	---	46
	10km/h ^b		49	50	32	31	38	52	53	---	46
TDX-SP TDX-SP2 Narrow	6km/h	ACS/LINX /Shark	25	26	34	33	39	52	53	---	46
	10km/h		49	50	32	31	38	52	53	---	46

^a used from 28.09.11 / ^b used from 07.09.11 / ^c used till 31.05.13 / ^d used from 01.06.13 (S/N 13F0600078)

Pos.	Part number	Description	Valid from → until	Qty
-	N23004	Feather key DIN6885-A-6x6x40		1
1	SP1524195	Propulsion cpl. LH, Gearbox i = 23:1, Motor, 2-pole, 250W, slow, 600mm cable		1
2	SP1524194	Propulsion cpl. RH, Gearbox i = 23:1, Motor, 2-pole, 250W, slow, 600mm cable		1
3	SP1541408	Propulsion cpl. LH, Gearbox i = 23:1, Motor, 2-pole, 250W, slow, 600mm cable, VR2		1
4	SP1541410	Propulsion cpl. RH, Gearbox i = 23:1, Motor, 2-pole, 250W, slow, 600mm cable, VR2		1
5	SP1541409	Propulsion cpl. LH, Gearbox i = 23:1, Motor, 2-pole, 250W, slow, 600mm cable, R-Net		1
6	SP1541411	Propulsion cpl. RH, Gearbox i = 23:1, Motor, 2-pole, 250W, slow, 600mm cable, R-Net		1
7	SP1524197	Propulsion cpl. LH, Gearbox i = 18:1, Motor, 2-pole, 250W, fast, 600mm cable		1
8	SP1524196	Propulsion cpl. RH, Gearbox i = 18:1, Motor, 2-pole, 250W, fast, 600mm cable		1
9	SP1541404	Propulsion cpl. LH, Gearbox i = 18:1, Motor, 2-pole, 250W, fast, 600mm cable, VR2		1
10	SP1541406	Propulsion cpl. RH, Gearbox i = 18:1, Motor, 2-pole, 250W, fast, 600mm cable, VR2		1
11	SP1541405	Propulsion cpl. LH, Gearbox i = 18:1, Motor, 2-pole, 250W, fast, 600mm cable, R-Net		1
12	SP1541407	Propulsion cpl. RH, Gearbox i = 18:1, Motor, 2-pole, 250W, fast, 600mm cable, R-Net		1
13	SP1168983	Propulsion cpl. LH, Gearbox i = 23:1, Motor, 2-pole, 250W, slow, 600mm cable		1
14	SP1168984	Propulsion cpl. RH, Gearbox i = 23:1, Motor, 2-pole, 250W, slow, 600mm cable		1
15	SP1168989	Propulsion cpl. LH, Gearbox i = 18:1, Motor, 4-pole, 340W, fast, 760mm cable		1
16	SP1168988	Propulsion cpl. RH, Gearbox i = 18:1, Motor, 4-pole, 340W, fast, 760mm cable		1
17	SP1568147	Propulsion cpl. LH, Gearbox i = 23:1, Motor, 2-pole, 250W, slow, 600mm cable		1

Overview Propulsion SSD


Pos.	Part number	Description	Valid from → until	Qty
18	SP1568148	Propulsion cpl. RH, Gearbox i = 23:1, Motor, 2-pole, 250W, slow, 600mm cable		1
19	SP1568149	Propulsion cpl. LH, Gearbox i = 18:1, Motor, 2-pole, 250W, fast, 600mm cable		1
20	SP1568150	Propulsion cpl. RH, Gearbox i = 18:1, Motor, 2-pole, 250W, fast, 600mm cable		1
21	SP1568151	Propulsion cpl. LH, Gearbox i = 23:1, Motor, 4-pole, 340W, slow, 760mm cable		1
22	SP1568152	Propulsion cpl. RH, Gearbox i = 23:1, Motor, 4-pole, 340W, slow, 760mm cable		1
23	SP1568153	Propulsion cpl. LH, Gearbox i = 18:1, Motor, 4-pole, 340W, fast, 760mm cable		1
24	SP1568208	Propulsion cpl. RH, Gearbox i = 18:1, Motor, 4-pole, 340W, fast, 760mm cable		1
25	SP1602974	Propulsion cpl. LH, Gearbox i = 23:1, Motor 4-pole 340W, slow, 760mm cable (incl. declutching lever)		1
26	SP1602975	Propulsion cpl. RH, Gearbox i = 23:1, Motor 4-pole 340W, slow, 760mm cable (incl. declutching lever)		1
27	SP1541366	Propulsion cpl. LH, Gearbox i = 18:1, Motor, 4-pole, 340W, fast, 760mm cable <i>S/N 061122610151 =></i>		1
28	SP1541365	Propulsion cpl. RH, Gearbox i = 18:1, Motor, 4-pole, 340W, fast, 760mm cable <i>S/N 061122610151 =></i>		1
29	SP1170307	Propulsion cpl. LH, Gearbox i = 23:1, Motor, 4-pole, 340W, fast, 760mm cable (incl. declutching lever)		1
30	SP1170306	Propulsion cpl. RH, Gearbox i = 23:1, Motor, 4-pole, 340W, fast, 760mm cable (incl. declutching lever)		1
31	SP1163413	Gearbox, LH, i = 18:1		1
32	SP1163412	Gearbox, RH, i = 18:1		1
33	SP1163415	Gearbox, LH, i = 23:1		1
34	SP1163414	Gearbox, RH, i = 23:1		1
35	SP1542953	Clamp band with clutch		1
36	SP1163099	Motor, 2-pole, 250W, fast, 600mm cable, ACS		1
37	SP1164046	Motor, 2-pole, 250W, slow, 600mm cable, ACS		1
38	SP1165863	Motor, 4-pole, 340W, fast, 760mm cable, ACS		1
39	SP1165865	Motor, 4-pole, 340W, slow, 760mm cable, ACS		1
40	SP1544840	Motor, 2-pole, 250W, fast, 600mm cable, VR2		1
41	SP1544796	Motor, 2-pole, 250W, slow, 600mm cable, R-Net		1
42	SP1544841	Motor, 2-pole, 250W, fast, 600mm cable, R-Net		1
43	SP1544795	Motor, 2-pole, 250W, slow, 600mm cable, VR2		1
45	SP1540358	Kit, brushes with caps, 2-pole (Set of 2)		1
46	SP1522967	Kit, brushes with caps, 4-pole (Set of 4)		1
47	SP1536022	Propulsion cpl. LH, Gearbox i = 23:1, Motor, 4-pole, 340W, slow, 760mm cable (incl. declutching knob) <i>S/N 061122812868 =></i>		1
48	SP1536021	Propulsion cpl. RH, Gearbox i = 23:1, Motor, 4-pole, 340W, slow, 760mm cable (incl. declutching knob) <i>S/N 061122812868 =></i>		1
49	SP1602976	Propulsion cpl. LH, Gearbox i = 18:1, Motor 4-pole 340W, fast, 760mm cable (incl. declutching lever)		1
50	SP1602998	Propulsion cpl. RH, Gearbox i = 18:1, Motor 4-pole 340W, fast, 760mm cable (incl. declutching lever)		1
51	SP1579595	Declutching knob SSD RH/LH (1 pc) <i>Single part. Consider to buy another one for the other side if required. - Replaces SP1168668, SP1168669 & SP1169005</i>		1
52	SP1168667	Declutching lever, cpl. - LH		1
53	SP1168666	Declutching lever, cpl. - RH		1
54	SP1552782	Propulsion cpl. LH, Gearbox i = 23:1, Motor 4-pole, 340W, slow, 760mm cable, ACS		1
55	SP1552783	Propulsion cpl. RH, Gearbox i = 23:1, Motor 4-pole, 340W, slow, 760mm cable, ACS		1

Overview Propulsion SSD

Pos.	Part number	Description	Valid from → until	Qty
56	SP1552872	Propulsion cpl. LH, Gearbox i = 18:1, Motor 4-pole, 340W, fast, 760mm cable, ACS		1
57	SP1552873	Propulsion cpl. RH, Gearbox i = 18:1, Motor 4-pole, 340W, fast, 760mm cable, ACS		1
58	SP1552832	Propulsion cpl. LH, Gearbox i = 23:1, Motor 4-pole, 340W, slow, 760mm cable, R-Net		1
59	SP1552833	Propulsion cpl. RH, Gearbox i = 23:1, Motor 4-pole, 340W, slow, 760mm cable, R-Net		1
60	SP1552836	Propulsion cpl. LH, Gearbox i = 18:1, Motor 4-pole, 340W, fast, 760mm cable, R-Net		1
61	SP1552837	Propulsion cpl. RH, Gearbox i = 18:1, Motor 4-pole, 340W, fast, 760mm cable, R-Net		1
62	SP1562498	Motor, 4-Pol, 340W, slow, 760mm cable, R-Net		1
63	SP1562499	Motor, 4-Pol, 340W, fast, 760mm cable, R-Net		1

Storm4 TTPlus

Pos.	Part number	Description	Valid from → until	Qty
-		GGB2 Motors		0

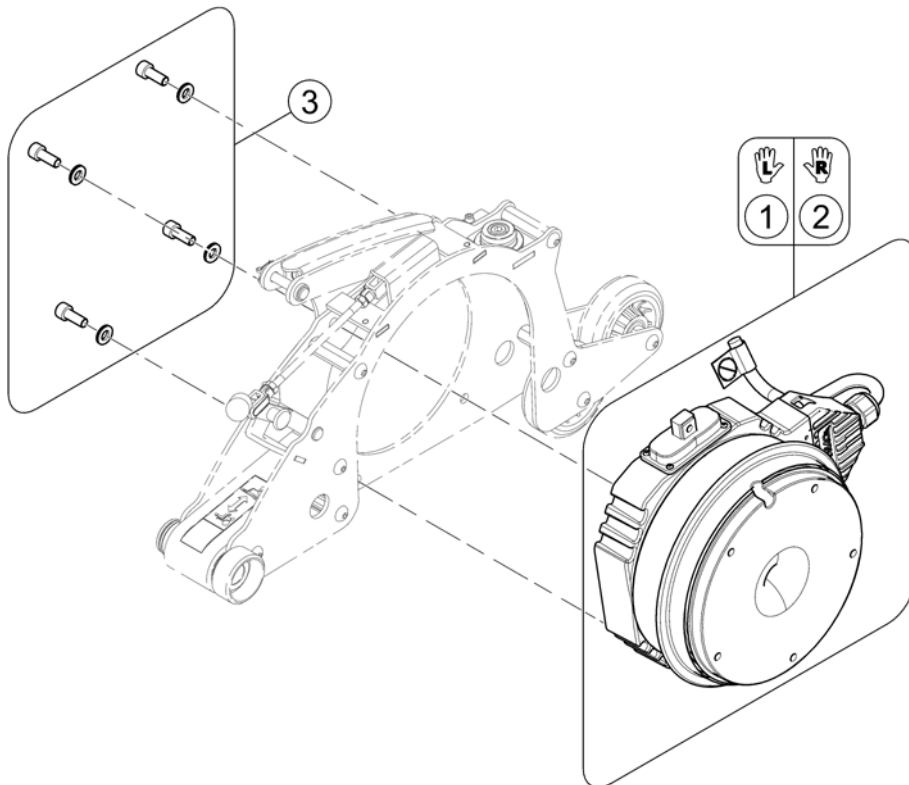



Pos.	Part number	Description	Valid from → until	Qty
1	SP1559356	GGB2 Motor LH		1
2	SP1559359	GGB2 Motor RH		1
3	SP1582231	Mounting Hardware GGB2 Motor		1

Electronic

ST4eeked iso







Pos.	Part number	Description	Valid from → until	Qty
1	-	Storm4		0
2	-	Storm4 TTPlus		0

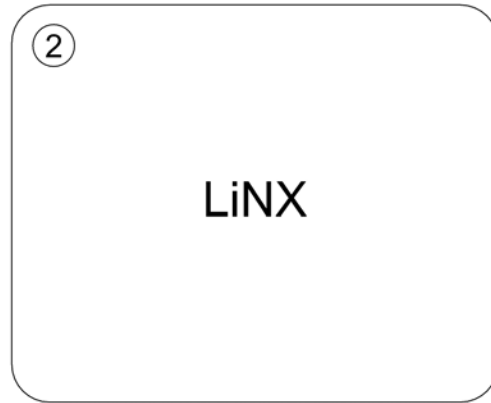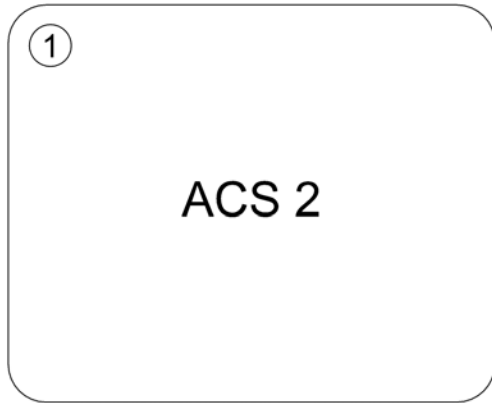

Pos.	Part number	Description	Valid from → until	Qty
1	-	ACS Electronic		1
2	-	LiNX Electronic		1

ACS Electronic

FDX79.iss

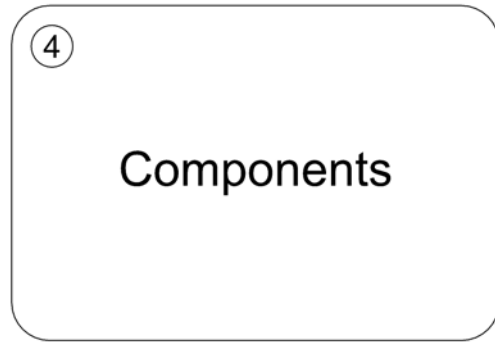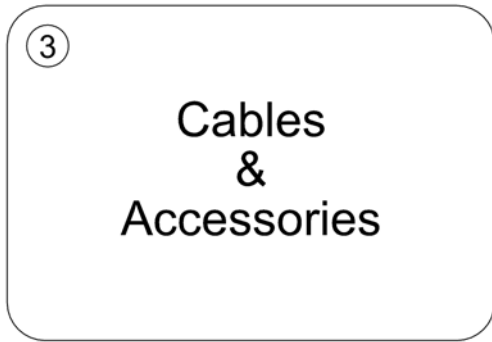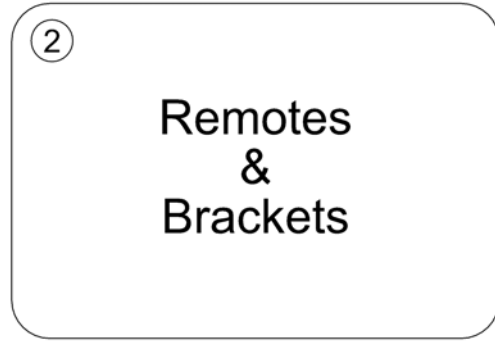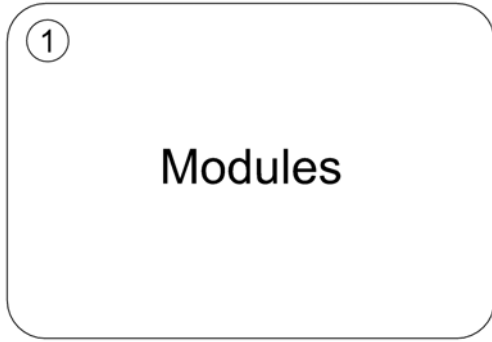


Pos.	Part number	Description	Valid from → until	Qty
1	-	ACS Electronic Modules		1
2	-	ACS Remotes & brackets		1
3	-	ACS Cables & Accessories		1
4	-	ACS Components		1

ACS Electronic Modules


Pos.	Part number	Description	Valid from → until	Qty
-		Power modules ACS 2		1
-		Holder Powermodule <i>NOT for Ultra Low Maxx Seat!</i>		1
-		Power module holder / Gyro <i>only with Ultra Low Maxx Seat!</i>		1
-		ACTs		1
-		Bracket ACT - Ultra Low Maxx Seat		1
-		G-Trac Sensor/Gyro		1
-		Holder Gyro		1
-		Light board		1

Power modules ACS 2

1524244X.dwg

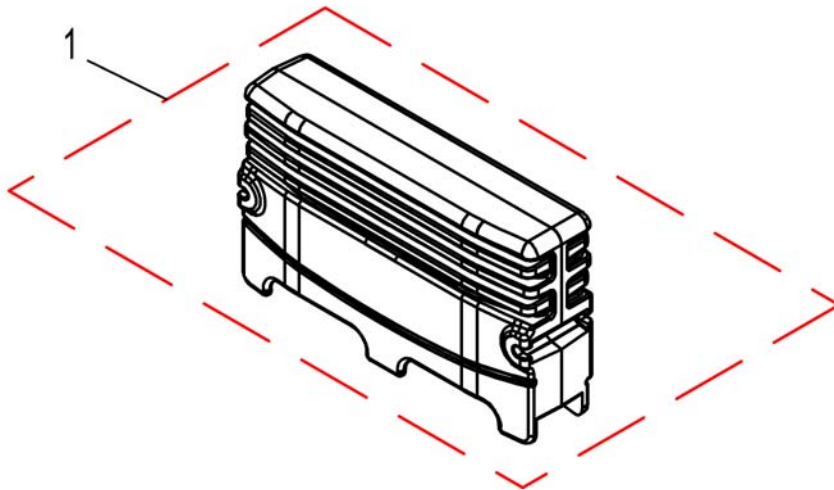


Pos.	Part number	Description	Valid from → until	Qty
-	SP1555702	Gasket kit for ACS2, Powermodule and ACT		1
1	SP1524244	Power Module ACS2, 90A, with light (DX2-PMA90L) <i>Lock washer between module and bracket!</i>		1
1	SP1537975	Power Module ACS2, 90A, with light and G-Trac (DX2-PMA90LG) <i>Lock washer between module and bracket!</i>		1

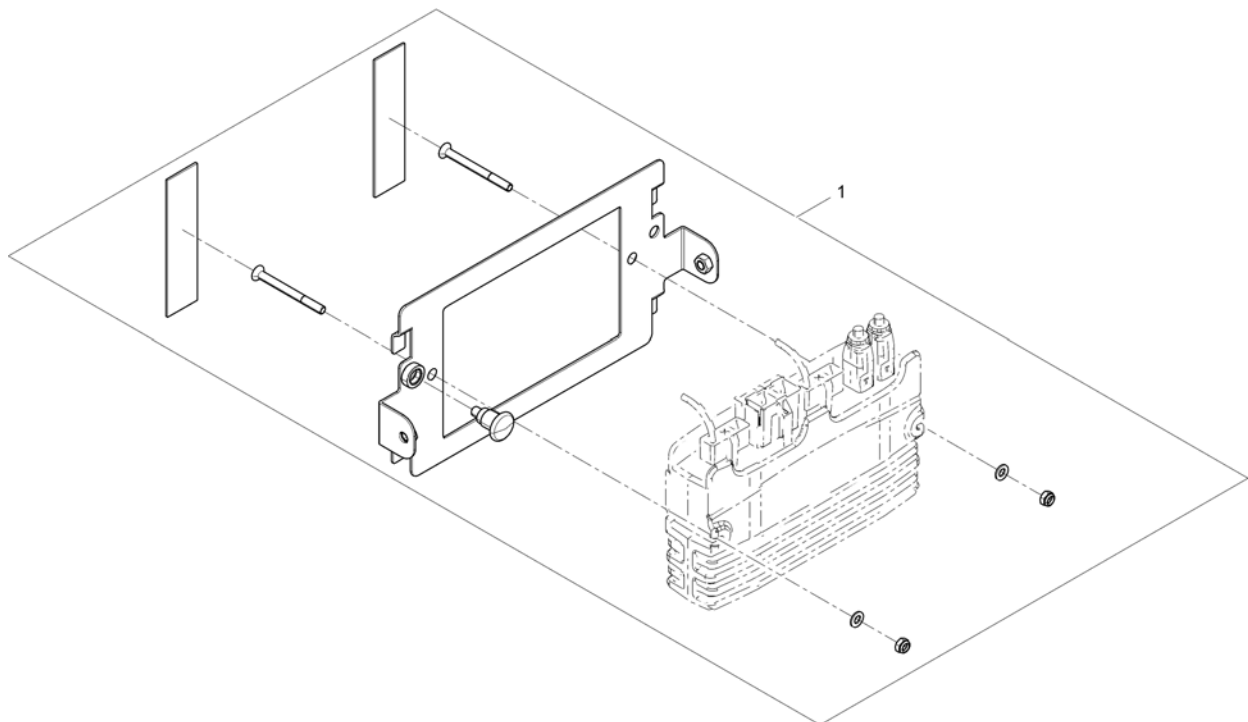

Pos.	Part number	Description	Valid from → until	Qty
1	SP1543129	Holder Powermodule <i>Lock washer between module and bracket!</i>		1

Power module holder / Gyro


154261X.kso

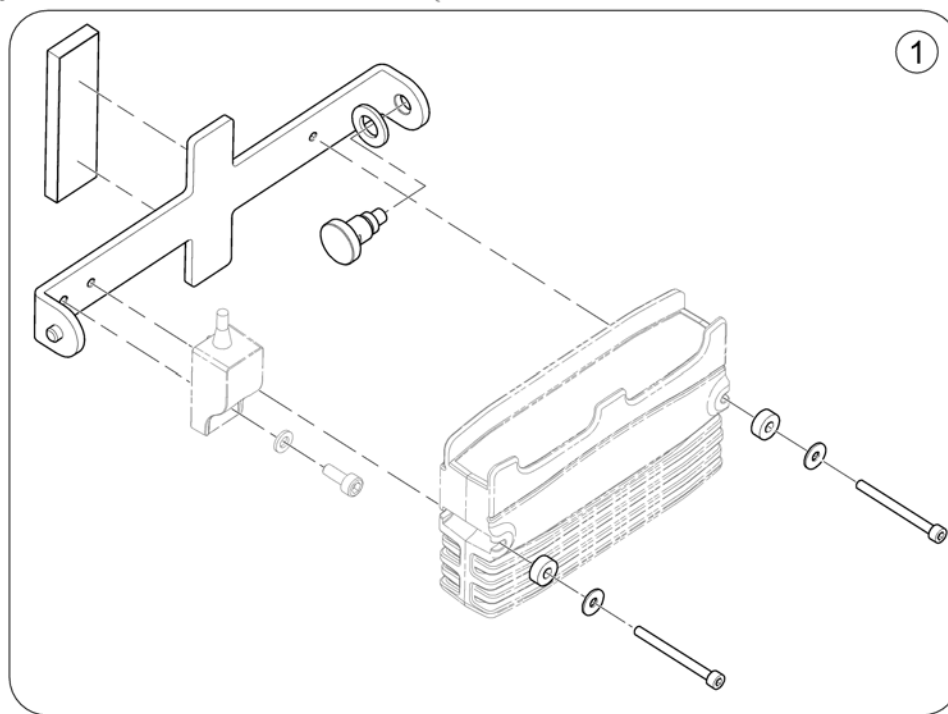

Pos.	Part number	Description	Valid from → until	Qty
1	SP1546769	Power module holder / Gyro <i>Lock washer between module and bracket!</i>		1

ACTs


1528542X.iso

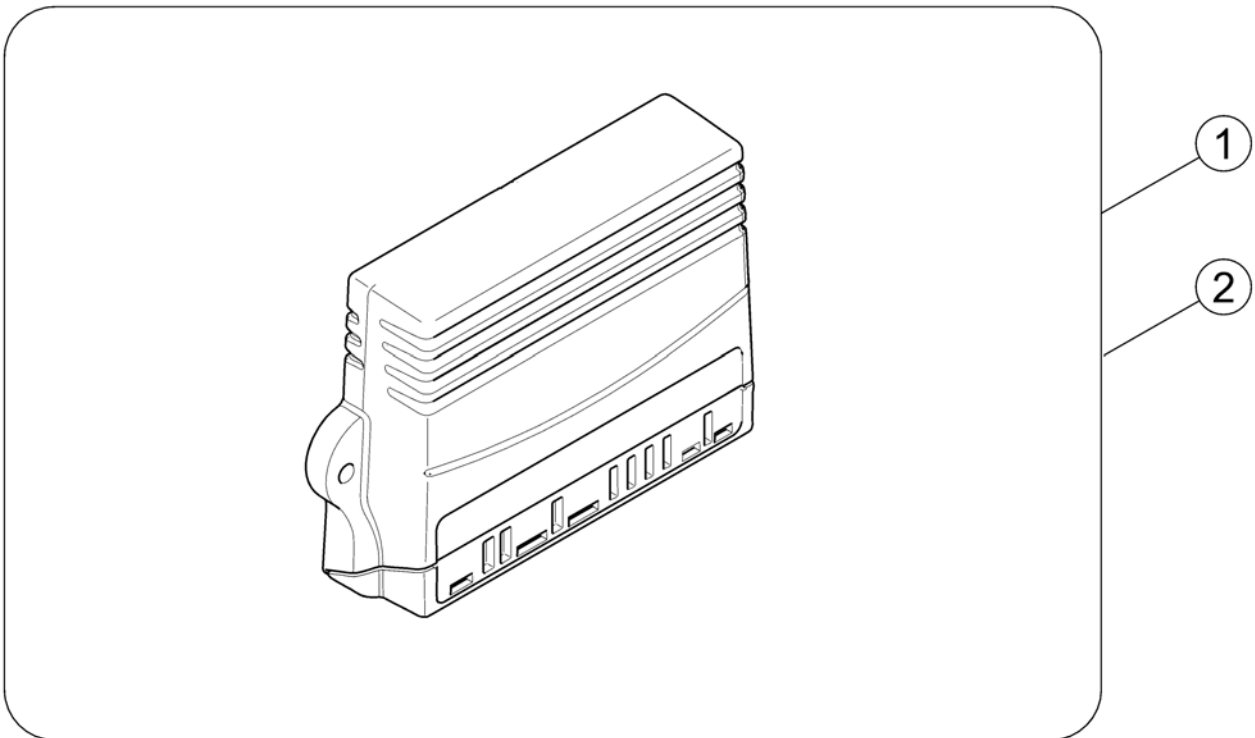

Pos.	Part number	Description	Valid from → until	Qty
1	SP1528542	ACT2 for ACS2, cpl.		1
2	SP1529382	ACT4 for ACS2, cpl.		1


Bracket ACT - Ultra Low Maxx Seat

1585063.iss

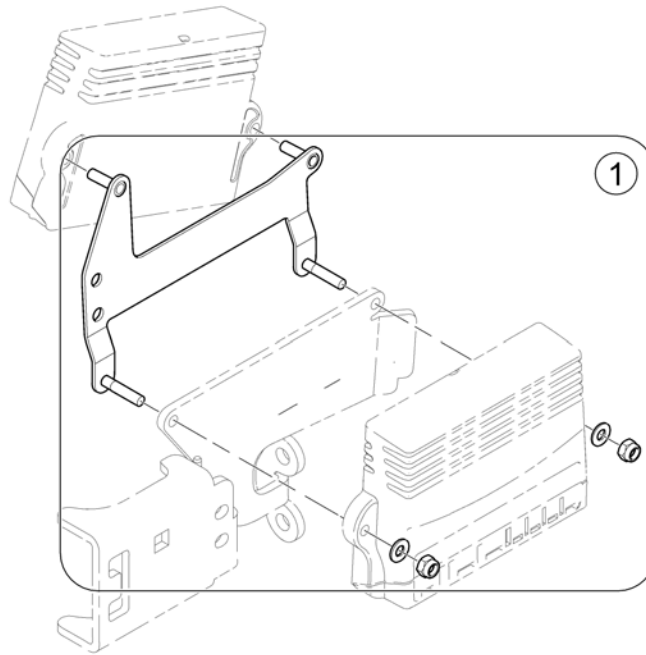

Pos.	Part number	Description	Valid from → until	Qty
1	SP1584343	Bracket ACTs (Ultra Low Maxx Seat)		1


1537848X.dwg

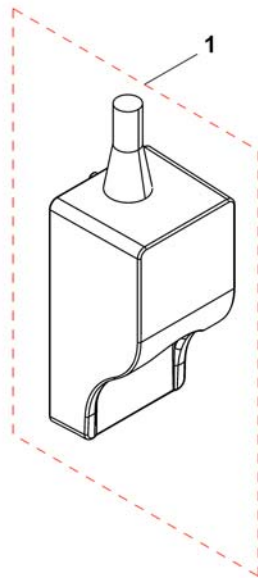

Pos.	Part number	Description	Valid from → until	Qty
1	SP1541535	G-Trac (Gyro) Sensor, cpl.		1

Holder Gyro



1537974X.dwg

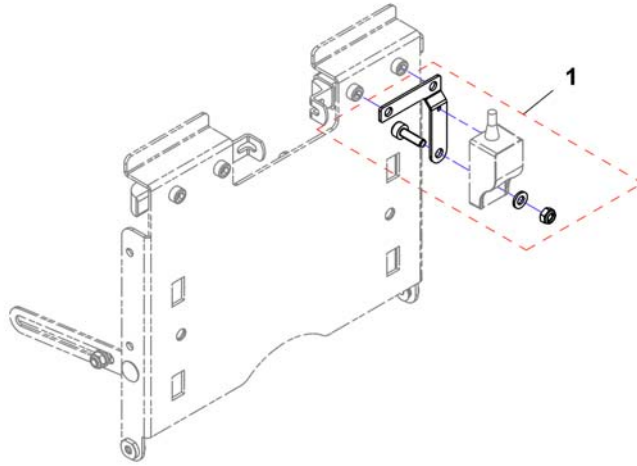

Pos.	Part number	Description	Valid from → until	Qty
1	SP1541625	Holder for Gyro		1


Light board

1528890X.dwg

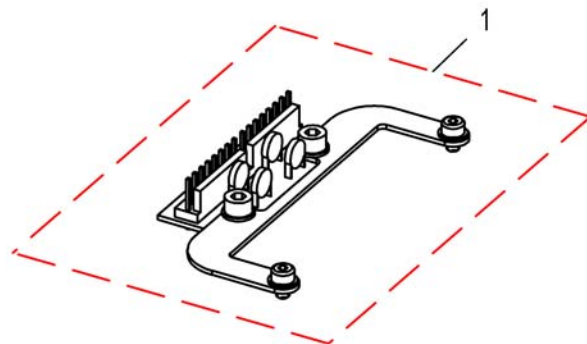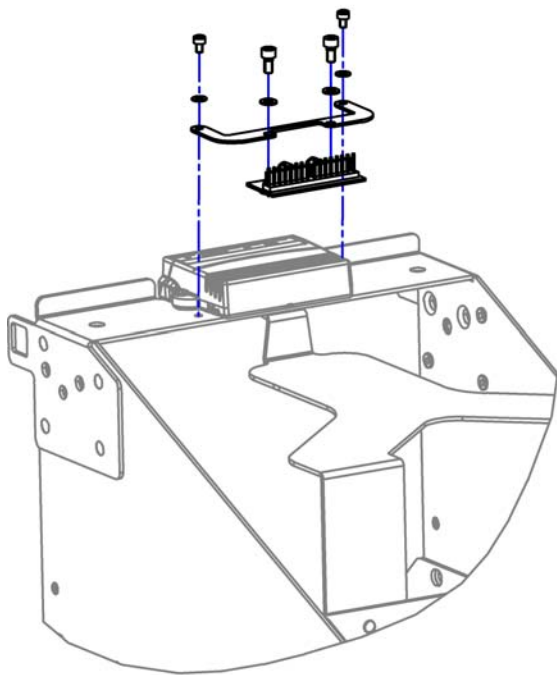


Pos.	Part number	Description	Valid from → until	Qty
-	SP1524237	Light cable ACS2		1
1	SP1528890	Light board, cpl.		1

ACS Remotes & brackets

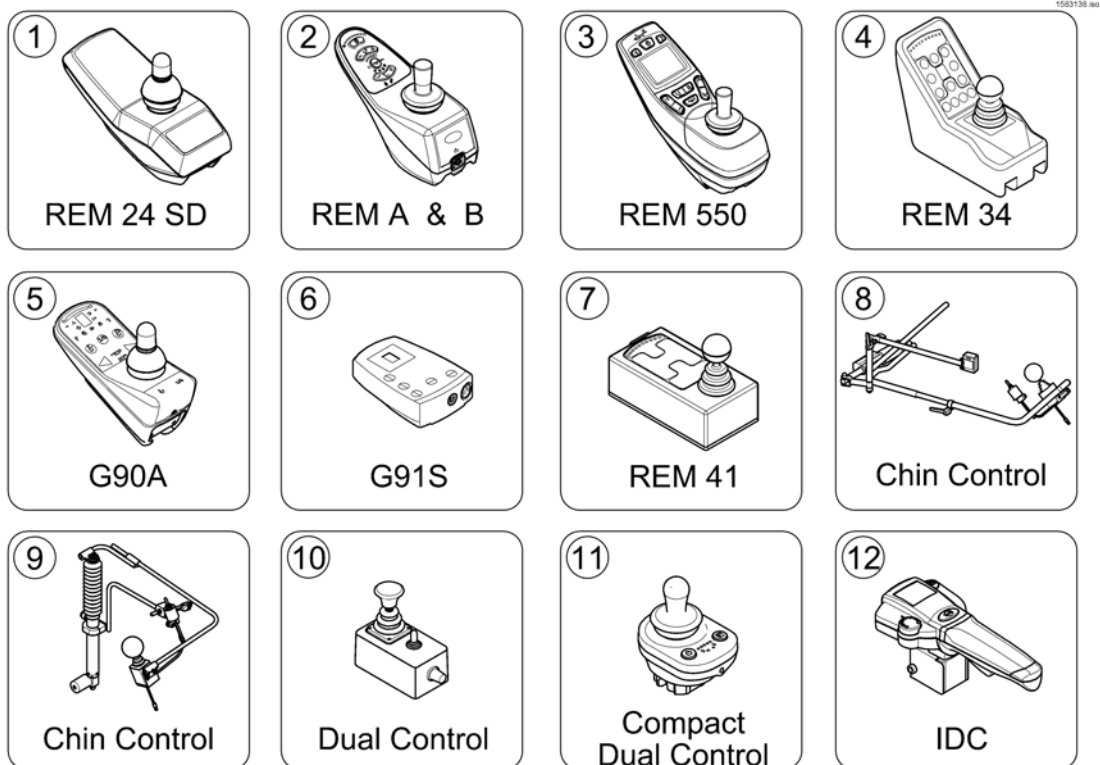


Pos.	Part number	Description	Valid from → until	Qty
1	-	REM 24 SD with brackets [1601/1602]		1
2	-	REM A & B with brackets [1638/1639/1643/1644]		1
3	-	REM 550 with brackets [1617/1618]		1
4	-	REM 34 with brackets [1611]		1
5	-	G90A with brackets [1614/1616]		1
6	-	G91S with brackets [1615]		1
7	-	Centre control REM 41 [1607/1608]		1
8	-	Chin Control, small (=> 16/12/31) [1609]		1
9	-	Chin Control - Modulite Seat (15/10/01 =>) [1719]		1
10	-	Dual Control with brackets [1291/1292]		1
11	-	Compact Dual Control with brackets		1
12	-	Intuitive Dual Control (IDC) with brackets [1645]		1


1602274 Rev

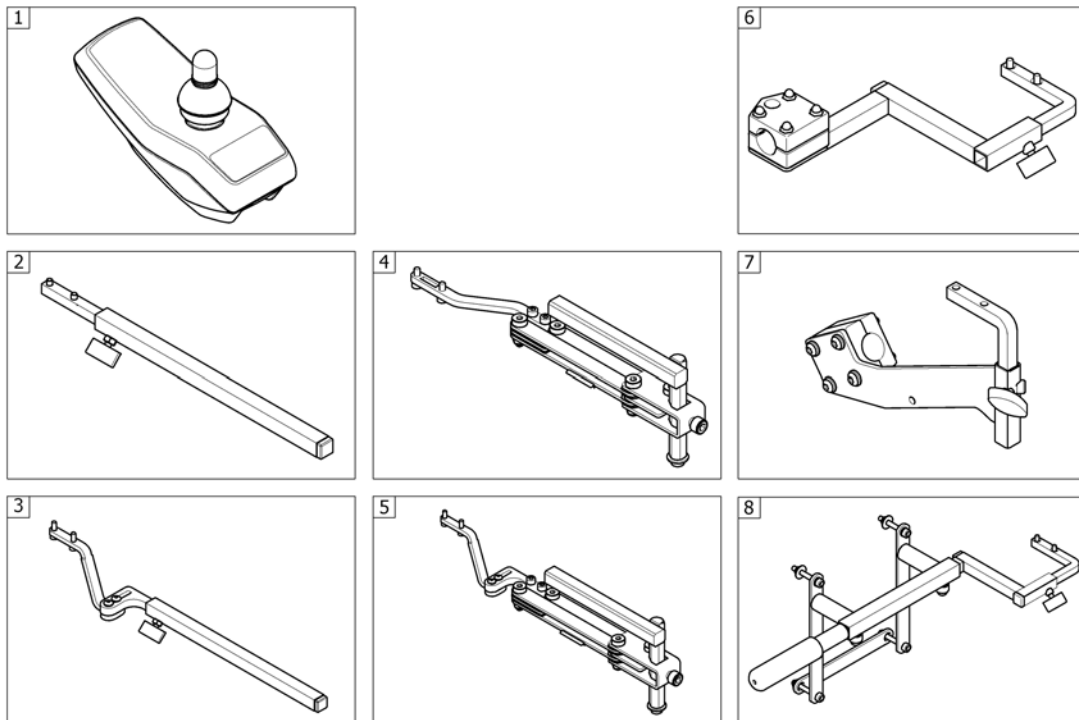

Pos.	Part number	Description	Valid from → until	Qty
1	-	Remote REM 24SD [1601/1602]		0
2	-	Remote Bracket Standard [1601+1207/1208]		1
3	-	Remote Bracket Standard - ULM and Soft Armpad on Modulite [1601+1207/1208+2020]		1
4	-	Remote Bracket Swing away & height adjust. [1601+1707/1708]		1
5	-	Remote Bracket Swing away & height adjust. - ULM and Soft Armpad on Modulite [1601+1707/1708+2020]		1
6	-	Remote Bracket Attendant [1602+1207/1208]		1
7	-	Remote Bracket Attendant Ultra Low Maxx Seat [1602+1207/1208+2020]		1
8	-	Remote Bracket Attendant for Recaro-Seat [1602+1207/1208]		1

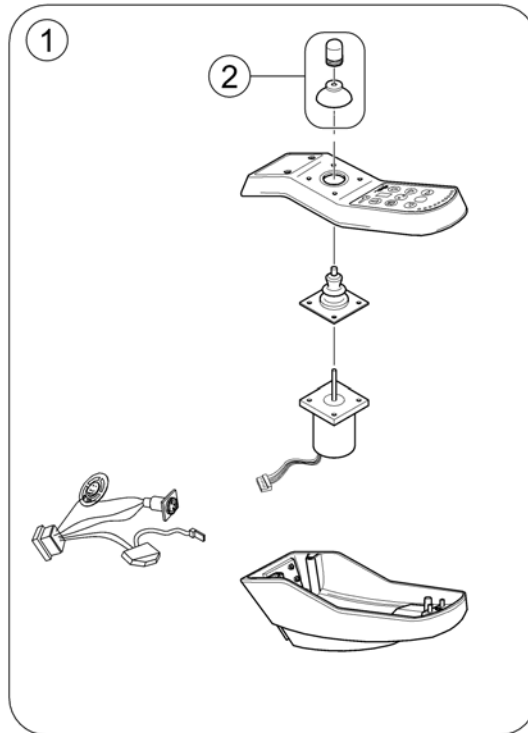


Pos.	Part number	Description	Valid from → until	Qty
1	1436541	Remote REM24SD		1
2	SPZ183229	Remote knob cpl.		1

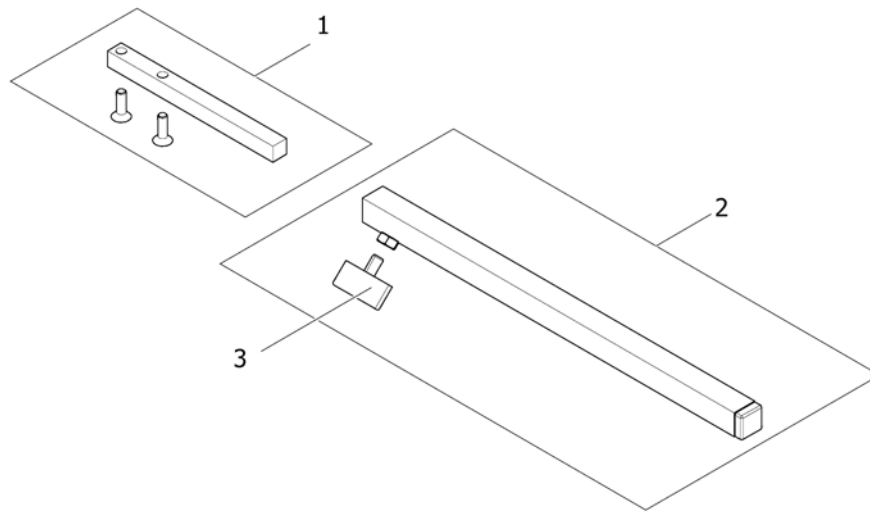

Pos.	Part number	Description	Valid from → until	Qty
1	SP1436542	Support, Standard		1
2	SPS183040	Remote Bracket, Standard		1
3	L31001	Wing ScREW M6X10		1

Remote Bracket Standard - ULM and Soft Armpad on Modulite


1588513.mo

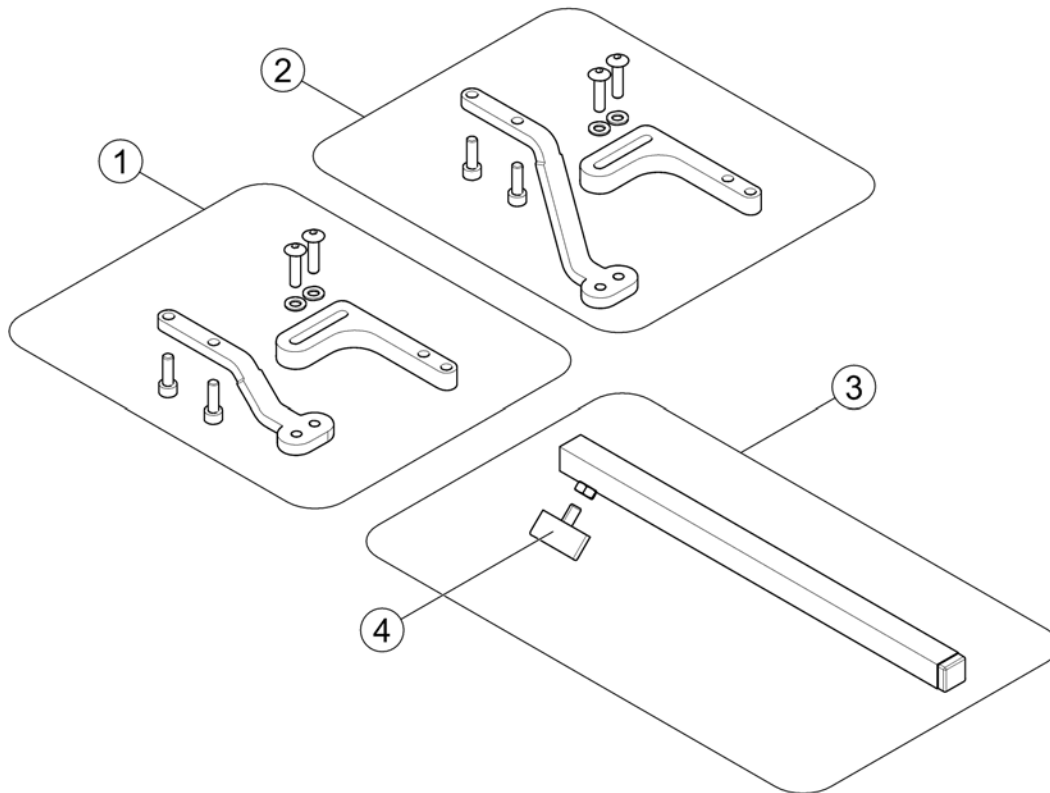

Pos.	Part number	Description	Valid from → until	Qty
1	SP1587309	Remote bracket Std. REM 24SD/A/B550		1
2	SP1587693	Remote bracket MACES Std - REM 24SD/A/B/550		1
3	SPS183040	Remote Bracket, Standard		1
4	L31001	Wing ScREW M6X10		1

Remote Bracket Swing away & height adjust.


1559852.iso

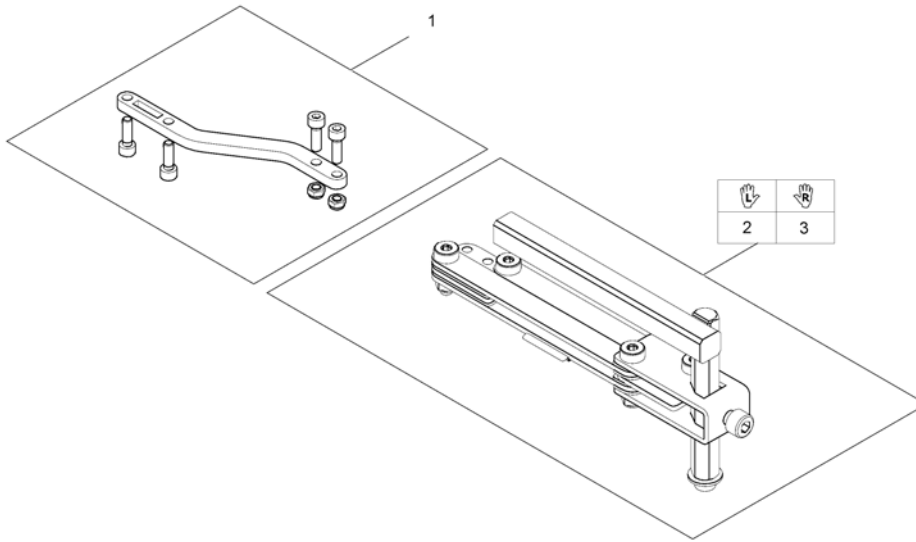

Pos.	Part number	Description	Valid from → until	Qty
1	SP1498222	Remote Support for Bracket height adjustable and swing away		1
2	SP1496249	Remote bracket, height adjustable & swing away, LH		1
3	SP1496248	Remote bracket, height adjustable & swing away, RH		1

Remote Bracket Swing away & height adjust. - ULM and Soft Armpad on Modulite


1588514.iso

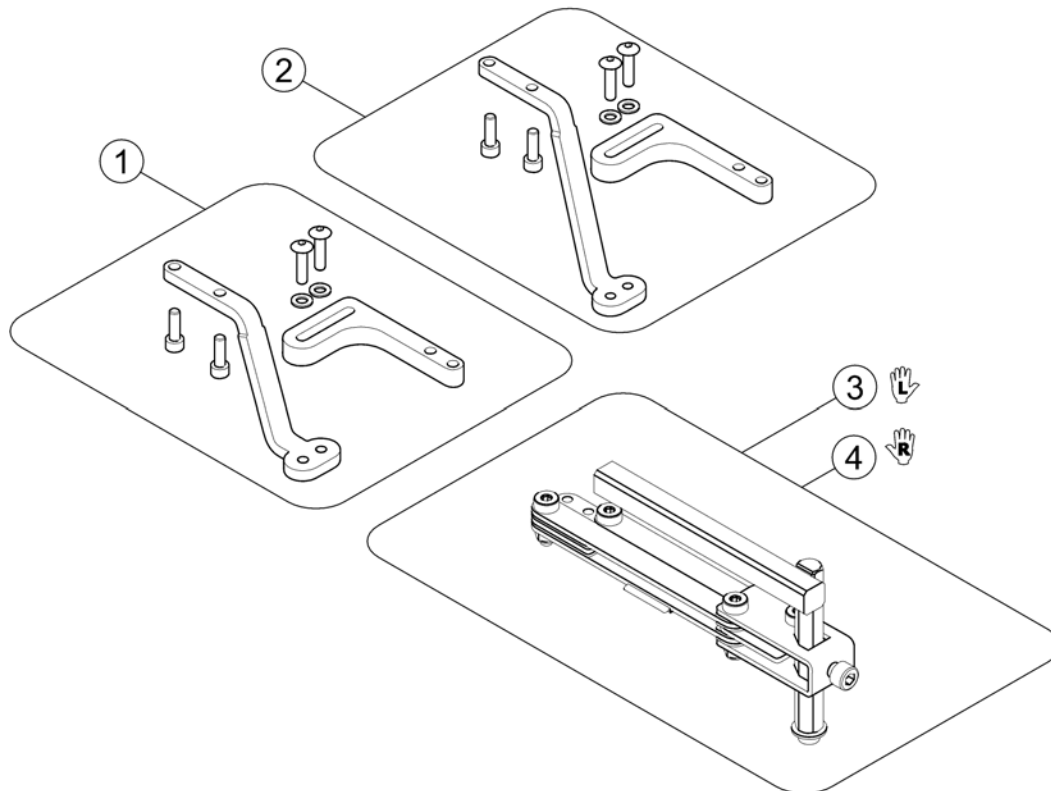

Pos.	Part number	Description	Valid from → until	Qty
1	SP1587329	Remote bracket swingaway REM 24SD/A/B/550		1
2	SP1587695	Remote bracket MACES swingaway - REM 24SD/A/B/550		1
3	SP1496249	Remote bracket, height adjustable & swing away, LH		1
4	SP1496248	Remote bracket, height adjustable & swing away, RH		1

Remote Bracket Attendant


1560837.00

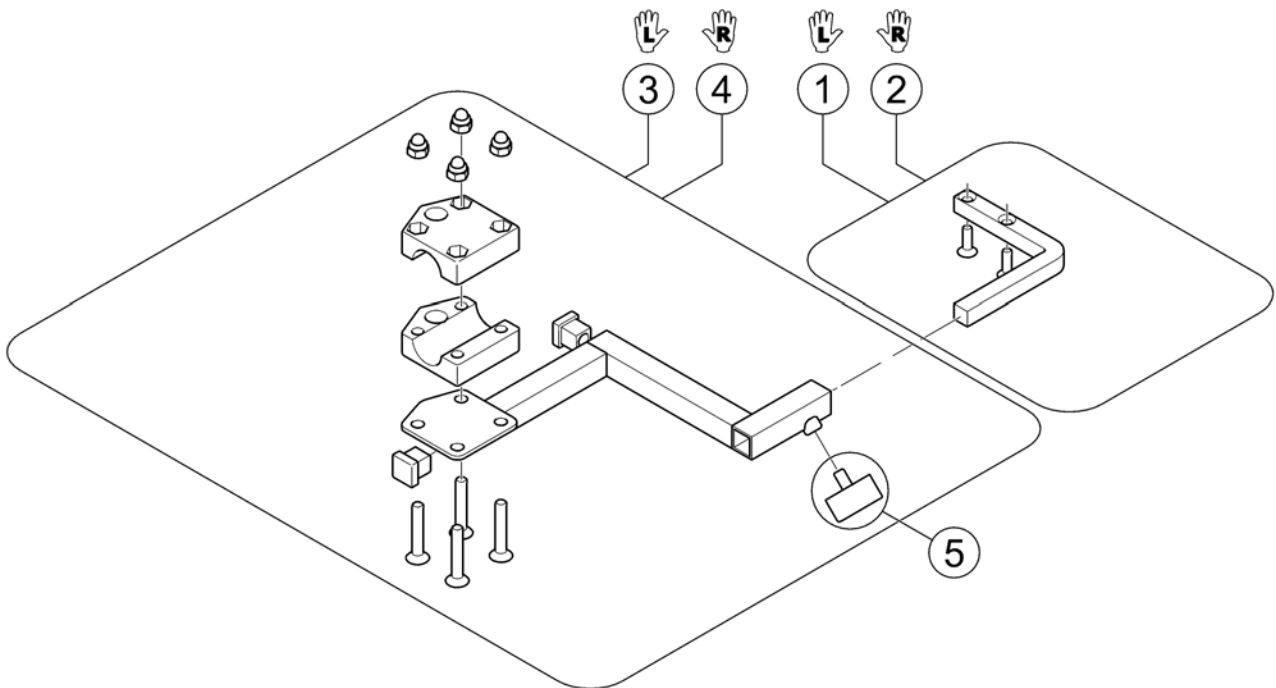

Pos.	Part number	Description	Valid from → until	Qty
1	1422806	Remote Support LH		1
2	1422810	Remote Support RH		1
3	SP1422937	Bracket for attendant control LH		1
4	SP1422936	Bracket for attendant control RH		1
5	L31001	Wing Scew M6X10		1

Remote Bracket Attendant Ultra Low Maxx Seat


1588437 8/0

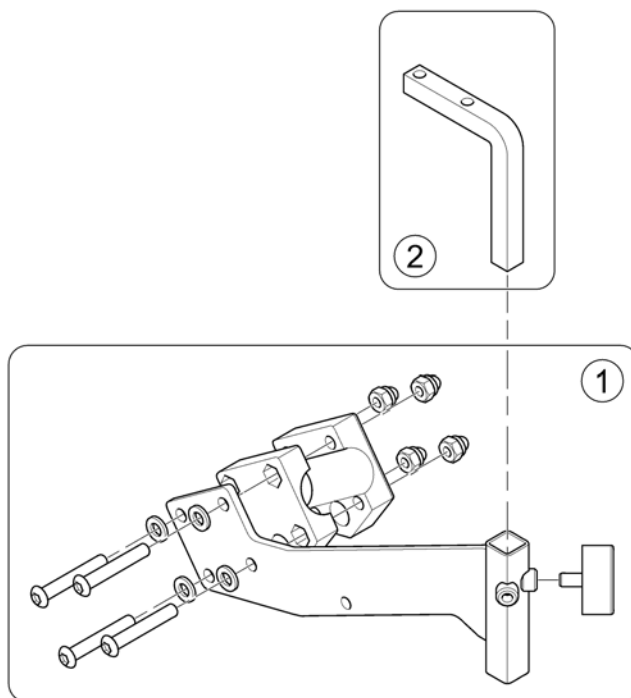

Pos.	Part number	Description	Valid from → until	Qty
1	SP1588427	Bracket Attendant Control		1
2	SP1587228	Adapter REM 24SD/A/B/550 (with screws)		1

Remote Bracket Attendant for Recaro-Seat


1563480.iso

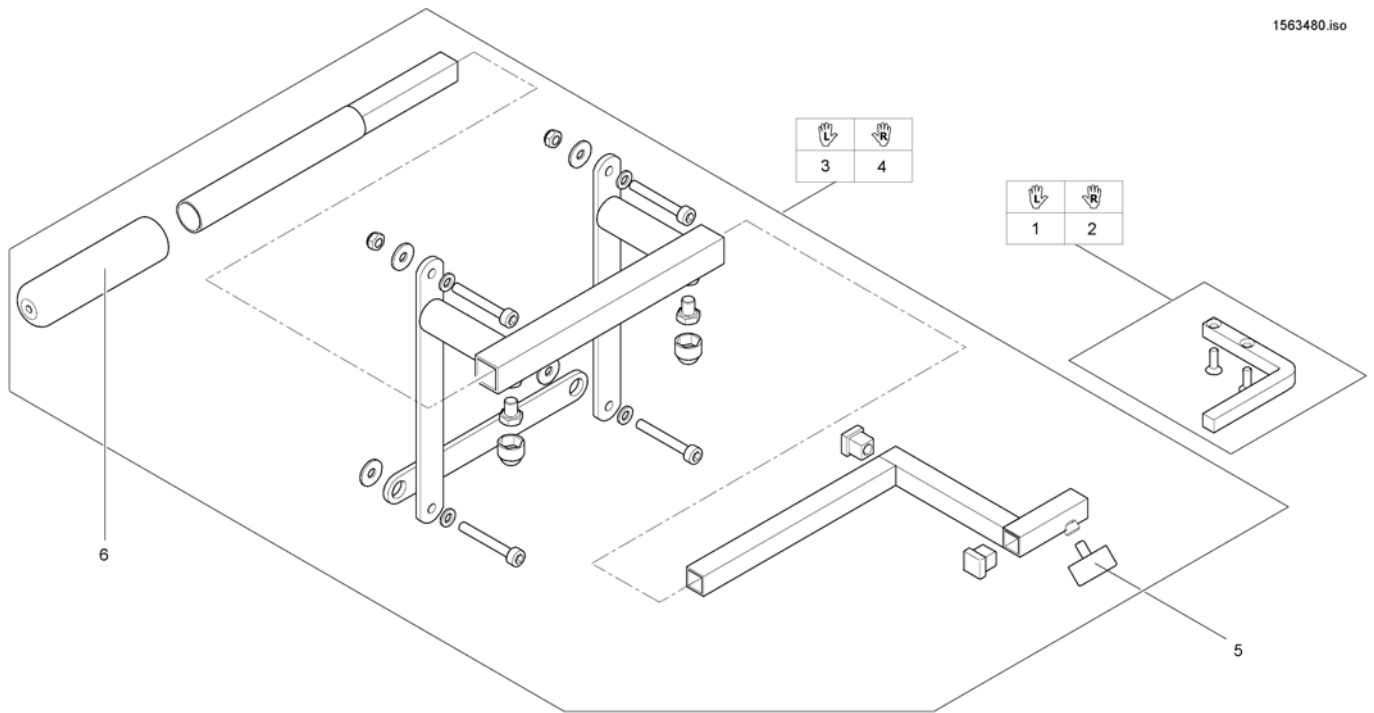

Pos.	Part number	Description	Valid from → until	Qty
1	1422806	Remote Support LH		1
2	1422810	Remote Support RH		1
3	SP1422814	Assebmly kit for dual control cpl., Recaro, LH		1
4	SP1422813	Assebmly kit for dual control cpl., Recaro, RH		1
5	L31001	Wing Scew M6X10		1
6	1435421	Handle plastic black Ø22 "Invacare"		1

REM A & B with brackets


1503649 no

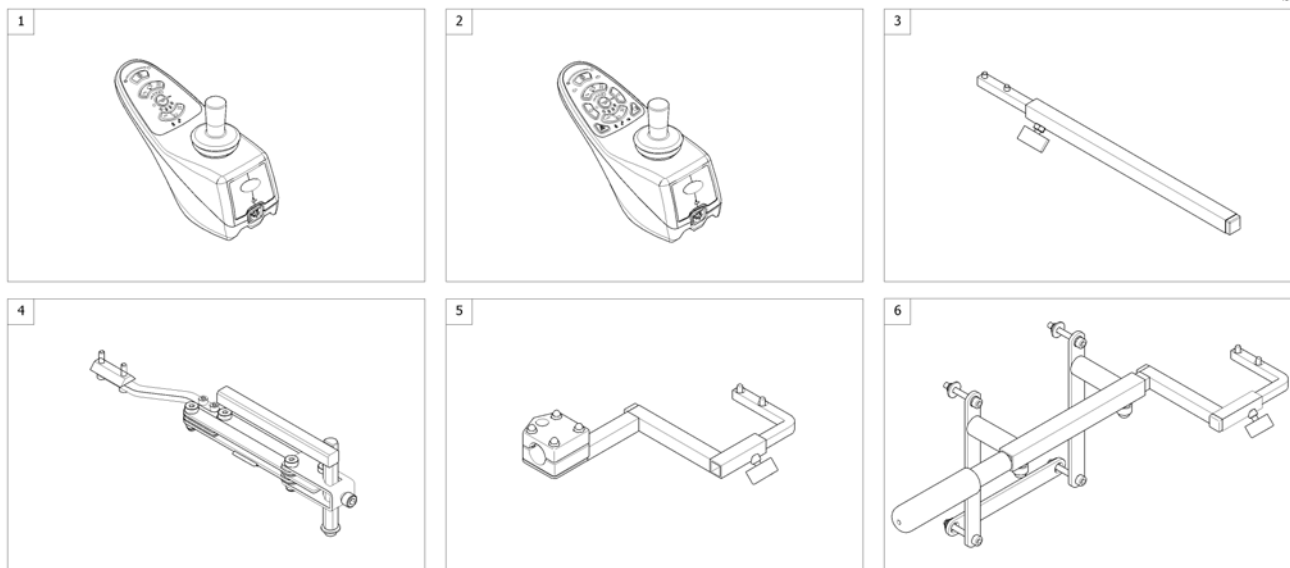

Pos.	Part number	Description	Valid from → until	Qty
1	SP1523440	Remote REM A [1638/1643]		1
2	SP1523445	Remote REM B [1639/1644]		1
3	-	Remote Bracket Standard [1638/1639+1207/1208]		1
4	-	Remote Bracket Swing away & height adjust. [1638/1639+1707/1708]		1
5	-	Remote Bracket Attendant for Standard-/EasyAdapt-Seat [1643/1644+1207/1208]		1
6	-	Remote Bracket Attendant for Recaro-Seat [1643/1644+1207/1208]		1

Remote Bracket Swing away & height adjust.


1559857.iso

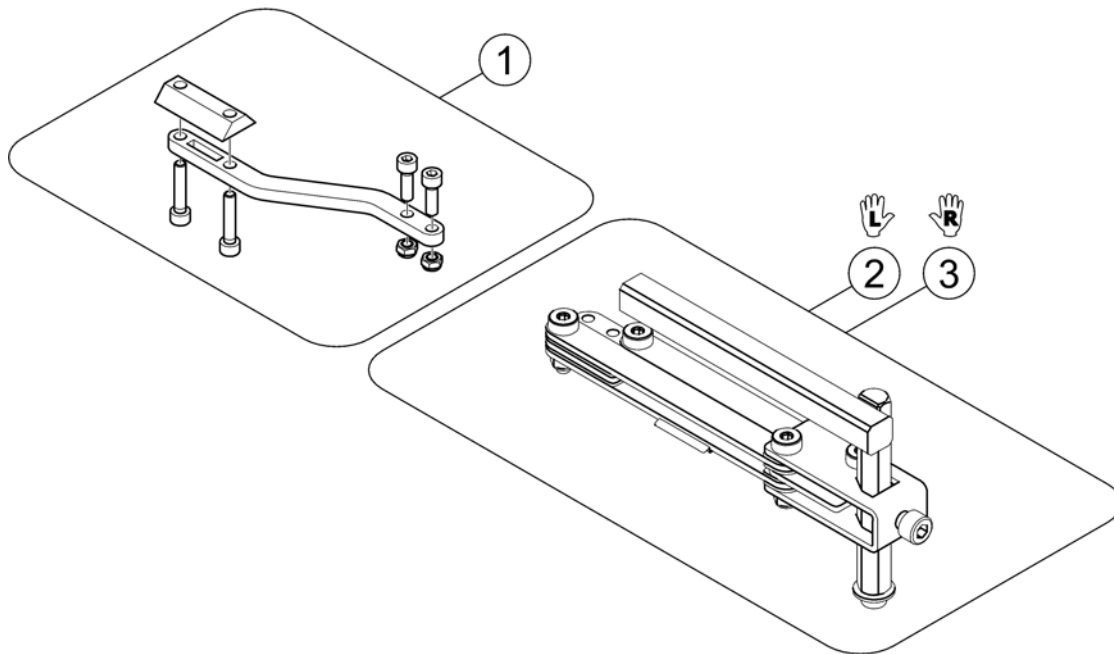


Pos.	Part number	Description	Valid from → until	Qty
1	SP1536180	Support for Remote Bracket, height adjustable & swing away		1
2	SP1496249	Remote bracket, height adjustable & swing away, LH		1
3	SP1496248	Remote bracket, height adjustable & swing away, RH		1

Remote Bracket Attendant for Standard-/EasyAdapt-Seat

1560837.00

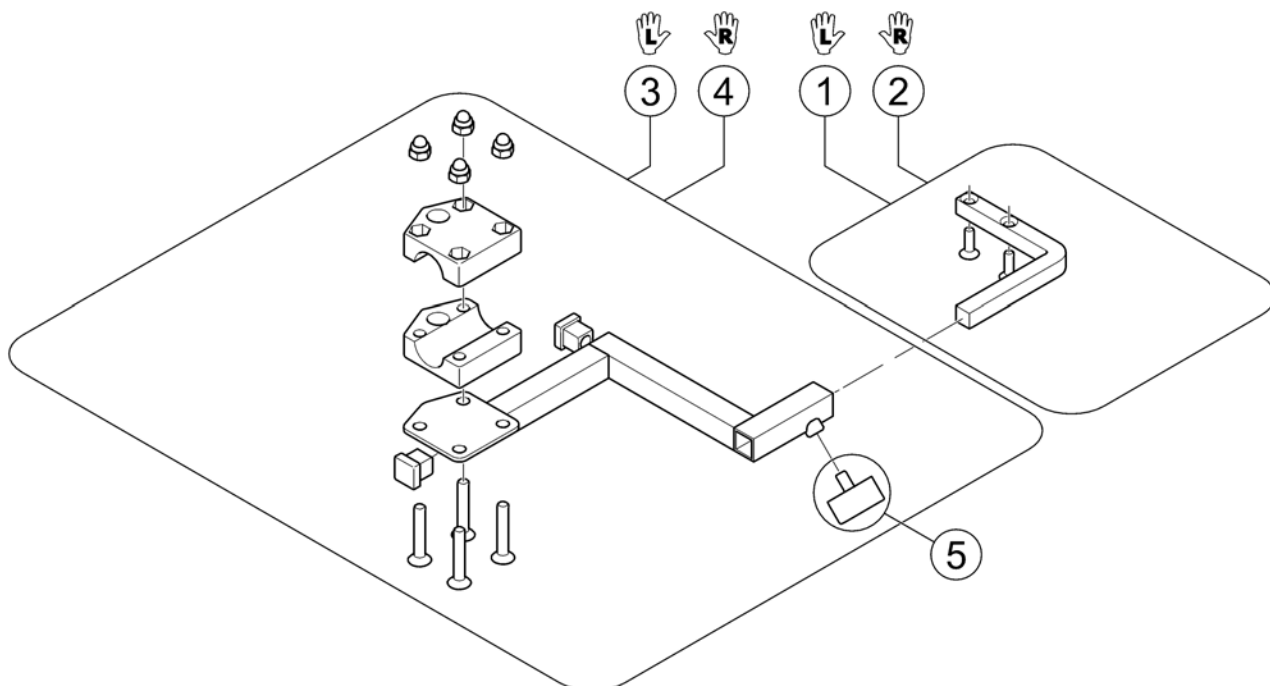


Pos.	Part number	Description	Valid from → until	Qty
1	1422806	Remote Support LH		1
2	1422810	Remote Support RH		1
3	SP1422937	Bracket for attendant control LH		1
4	SP1422936	Bracket for attendant control RH		1
5	L31001	Wing Scew M6X10		1

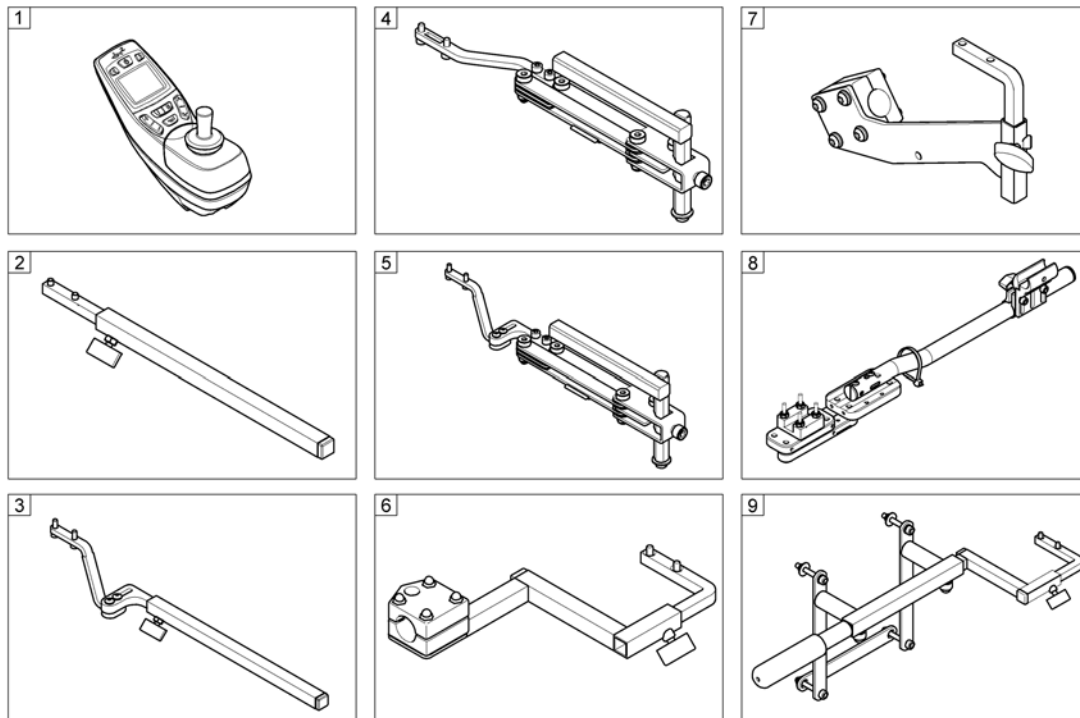



Pos.	Part number	Description	Valid from → until	Qty
1	SP1523444	Remote REM550 <i>[1617/1618]</i>		1
2	-	Remote Bracket Standard <i>[1617+1207/1208]</i>		1
3	-	Remote Bracket Standard - ULM and Soft Armpad on Modulite <i>[1617+1207/1208+2020]</i>		1
4	-	Remote Bracket Swing away & height adjust. <i>[1617+1707/1708]</i>		1
5	-	Remote Bracket Swing away & height adjust. - ULM and Soft Armpad on Modulite <i>[1617+1707/1708+2020]</i>		1
6	-	Remote Bracket Attendant <i>[1618+1207/1208]</i>		1
7	-	Remote Bracket Attendant Ultra Low Maxx Seat <i>[1618+1207/1208+2020]</i>		1
8	-	Remote Bracket Swing away Quad Link Ultra Low Maxx Seat <i>[1617+2075/2076+2020]</i>		1
9	-	Remote Bracket Attendant for Recaro-Seat <i>[1618+1207/1208]</i>		1

Remote Bracket Swing away Quad Link Ultra Low Maxx Seat

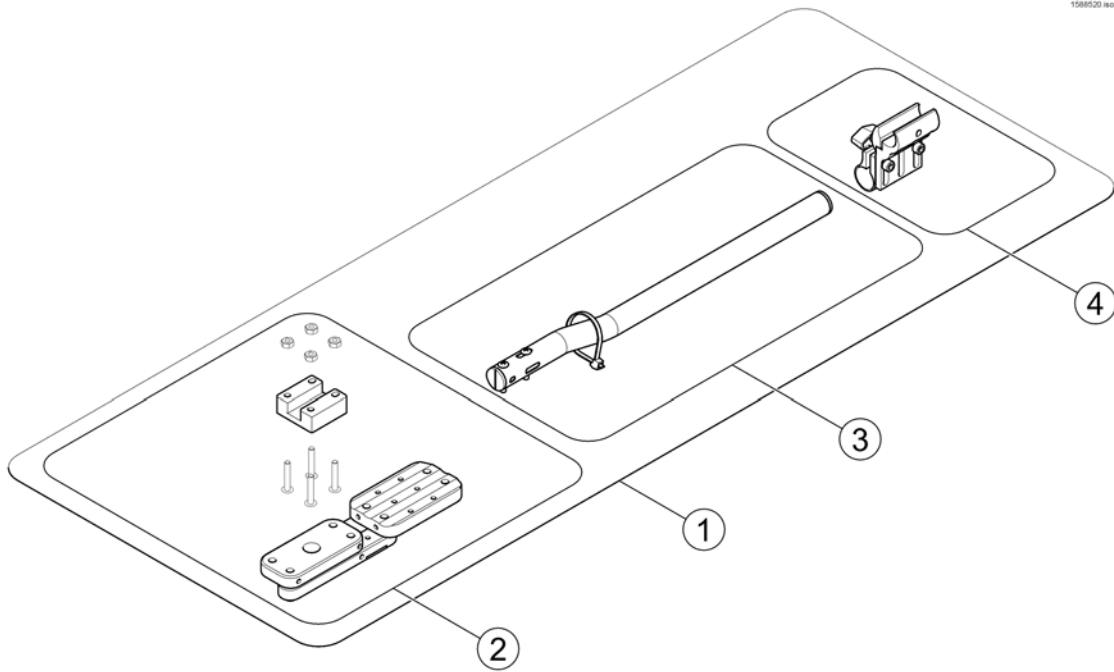

Pos.	Part number	Description	Valid from → until	Qty
1	SP1584346	Quad Link, cpl.		1
2	SP1140896	Quad Link		1
3	SP1139416	Quad Link bracket		1
4	SPTRC2089	Joystick Clamp Assy Quad Link		1


1560846 iso

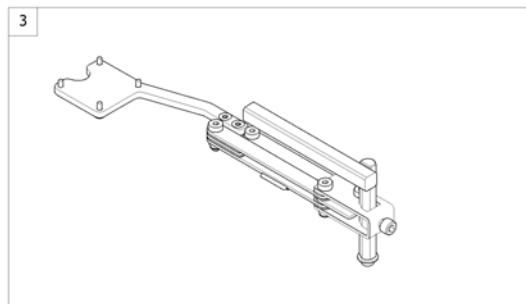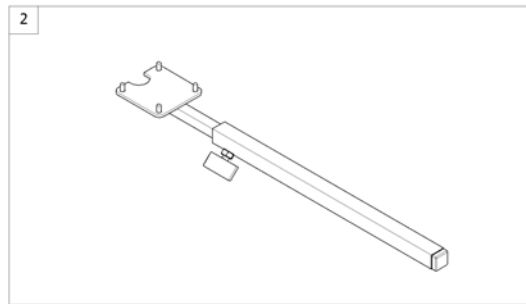

Pos.	Part number	Description	Valid from → until	Qty
1	SP1440677	Remote REM 34 [1611]		1
2	-	Remote Bracket Standard [1611+1207/1208]		1
3	-	Remote Bracket Swing away & height adjust. [1611+1707/1708]		1

Remote Bracket Standard


1560844.iso

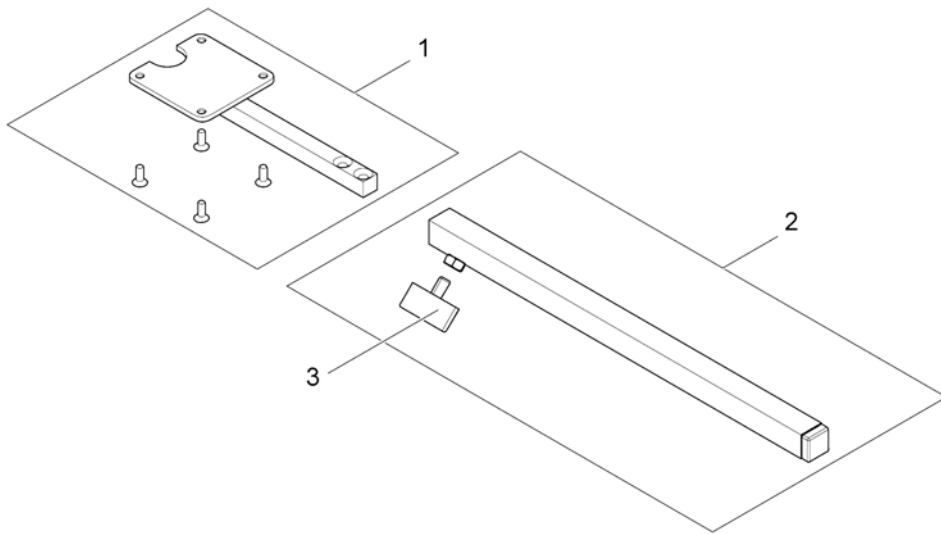

Pos.	Part number	Description	Valid from → until	Qty
1	SP1434921	Remote Support Standard		1
2	SPS183040	Remote Bracket, Standard		1
3	L31001	Wing ScREW M6X10		1

Remote Bracket Swing away & height adjust.


1560845.iso

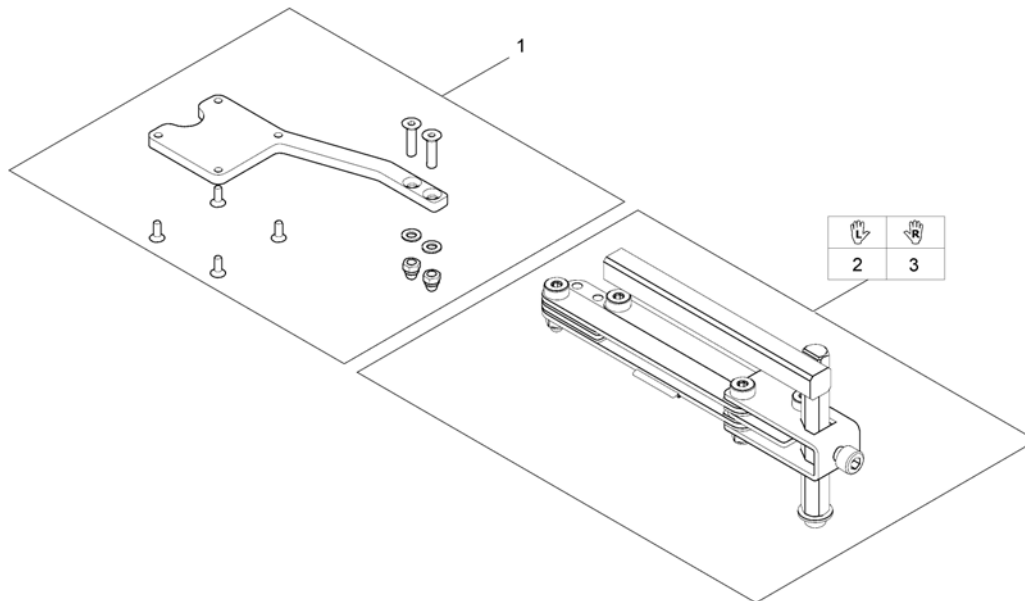


Pos.	Part number	Description	Valid from → until	Qty
1	SP1527125	Remote Support for Bracket height adjustable and swing away		1
2	SP1496249	Remote bracket, height adjustable & swing away, LH		1
3	SP1496248	Remote bracket, height adjustable & swing away, RH		1

G90A with brackets

1602276 iso

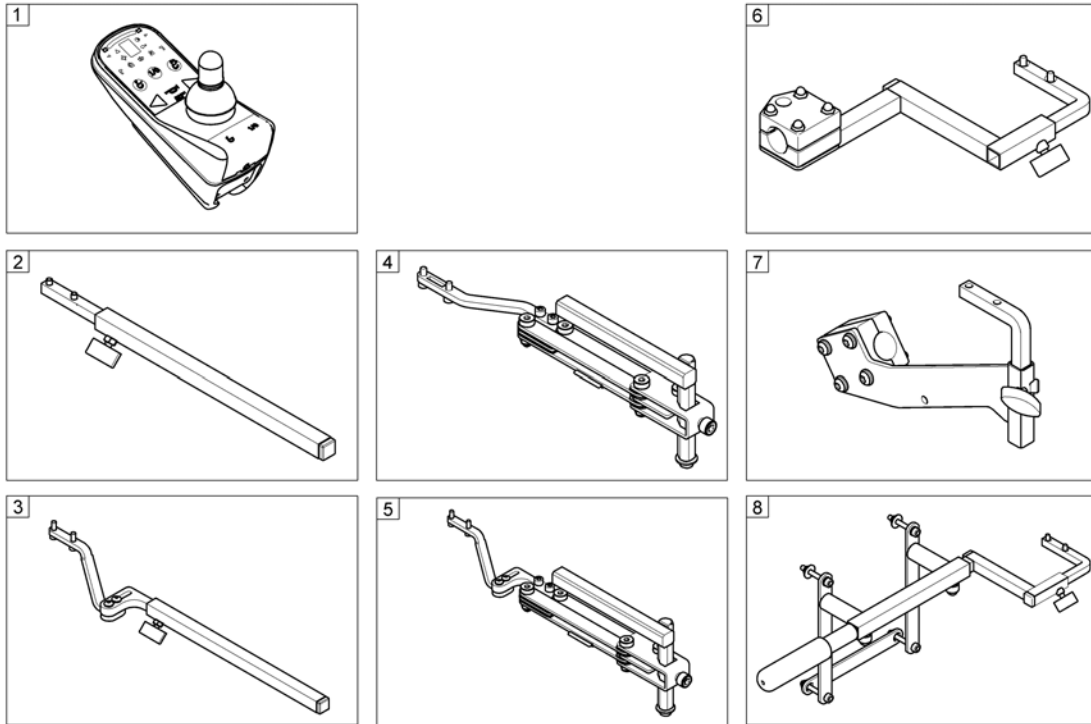


Pos.	Part number	Description	Valid from → until	Qty
1	SP1504720	Remote G90A <i>[1614/1616]</i>		1
2	-	Remote Bracket Standard <i>[1614+1207/1208]</i>		1
3	-	Remote Bracket Standard - ULM and Soft Armpad on Modulite <i>[1614+1207/1208+2020]</i>		1
4	-	Remote Bracket Swing away & height adjust. <i>[1614+1707/1708]</i>		1
5	-	Remote Bracket Swing away & height adjust. - ULM and soft Armpad on Modulite <i>[1614+1707/1708+2020]</i>		1
6	-	Remote Bracket Attendant <i>[1616+1207/1208]</i>		1
7	-	Remote Bracket Attendant Ultra Low Maxx Seat <i>[1616+1207/1208+2020]</i>		1
8	-	Remote Bracket Attendant for Recaro-Seat <i>[1618+1207/1208]</i>		1

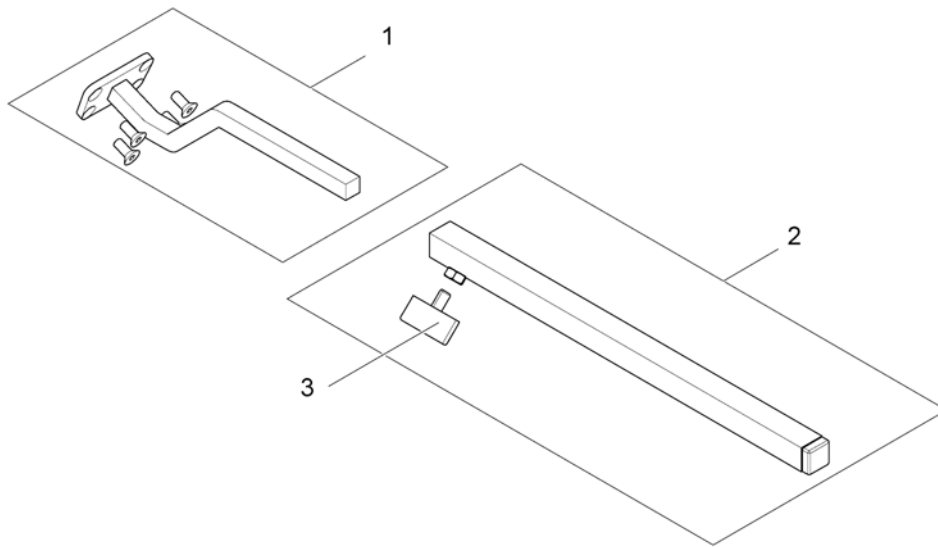

Pos.	Part number	Description	Valid from → until	Qty
1	SP1504822	Remote Support Standard		1
2	SPS183040	Remote Bracket, Standard		1
3	L31001	Wing ScREW M6X10		1

Remote Bracket Standard - ULM and Soft Armpad on Modulite


1588518.iso

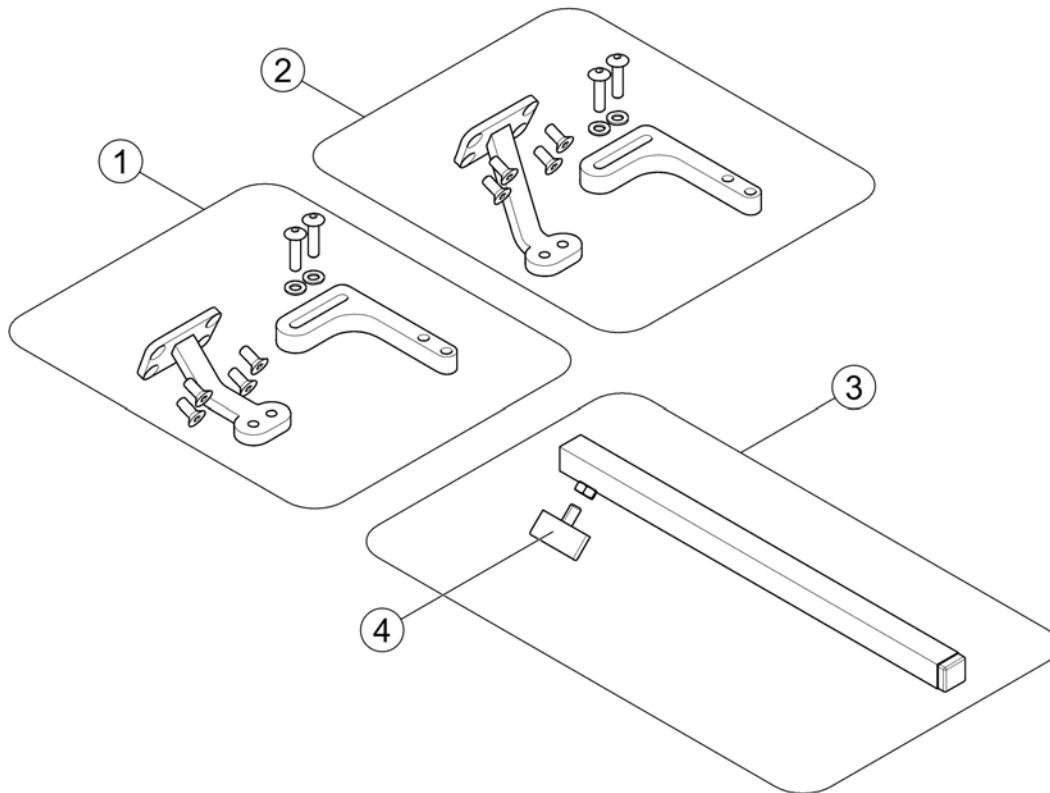

Pos.	Part number	Description	Valid from → until	Qty
1	SP1587310	Remote bracket Std G90		1
2	SP1587694	Remote bracket MACES Std - G90		1
3	SPS183040	Remote Bracket, Standard		1
4	L31001	Wing ScREW M6X10		1

Remote Bracket Swing away & height adjust.


1560310.iso

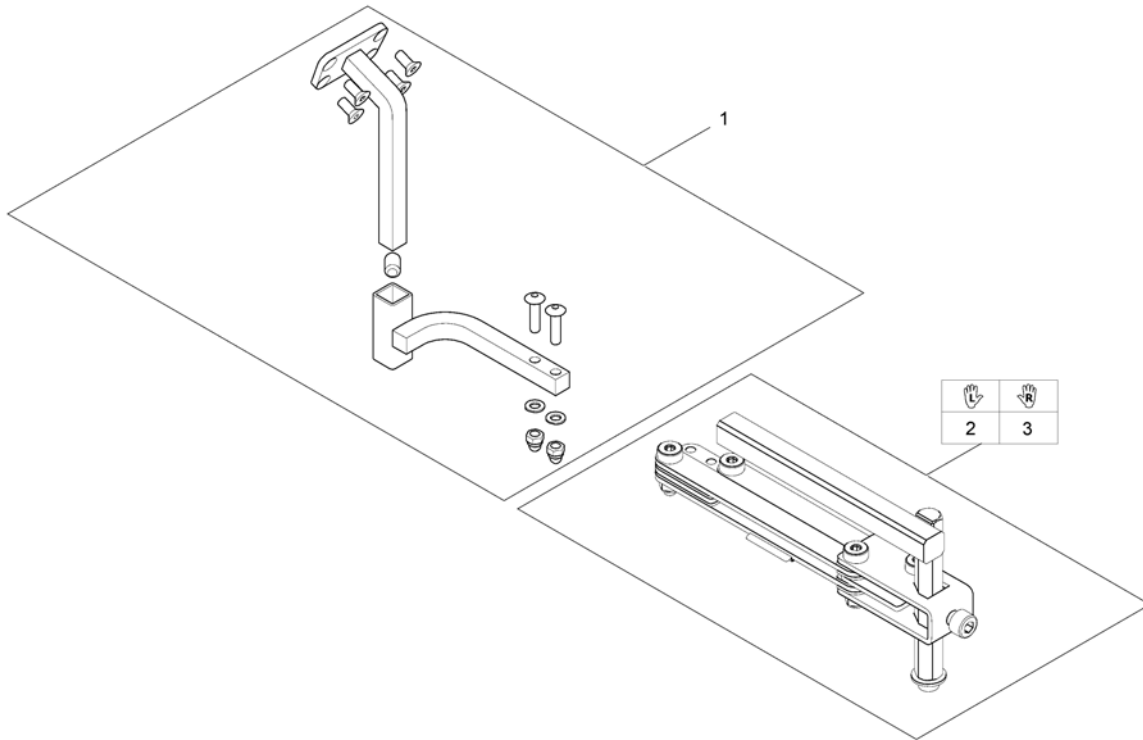

Pos.	Part number	Description	Valid from → until	Qty
1	SP1517352	Remote Support for Bracket height adjustable and swing away		1
2	SP1496249	Remote bracket, height adjustable & swing away, LH		1
3	SP1496248	Remote bracket, height adjustable & swing away, RH		1

Remote Bracket Swing away & height adjust. - ULM and soft Armpad on Modulite


1588519.mo

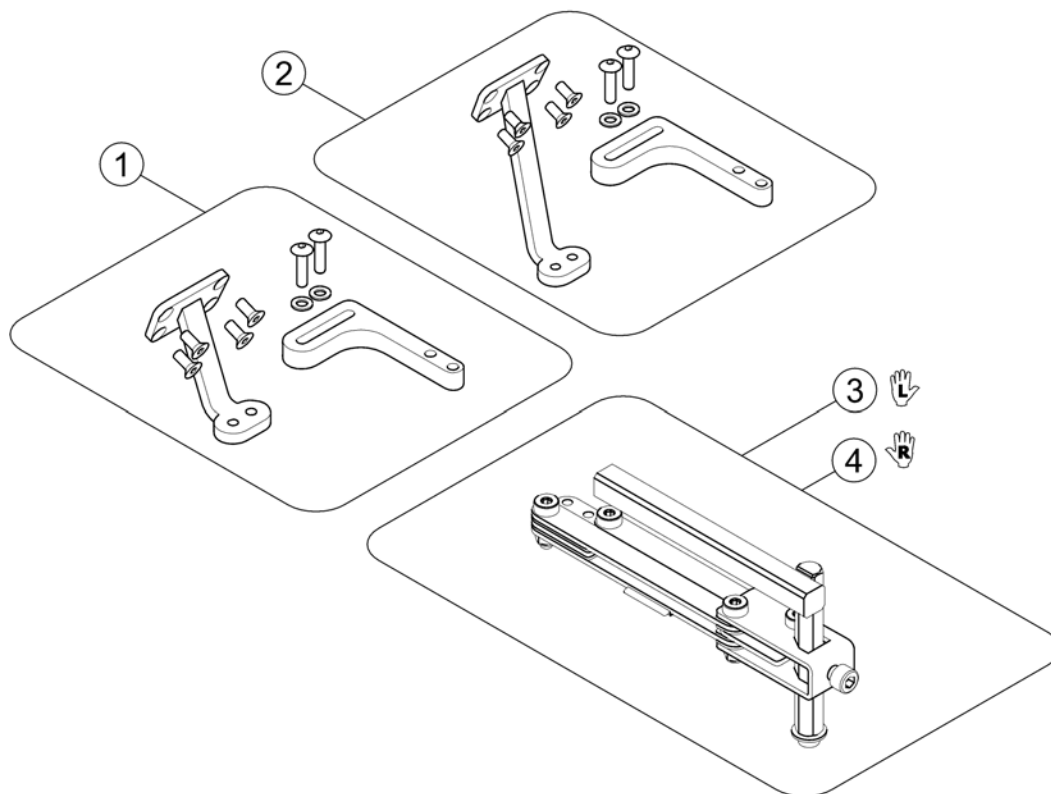

Pos.	Part number	Description	Valid from → until	Qty
1	SP1587331	Remote bracket swingaway G90		1
2	SP1587696	Remote bracket MACES, swingaway - G90		1
3	SP1496249	Remote bracket, height adjustable & swing away, LH		1
4	SP1496248	Remote bracket, height adjustable & swing away, RH		1

Remote Bracket Attendant


1560853.iso

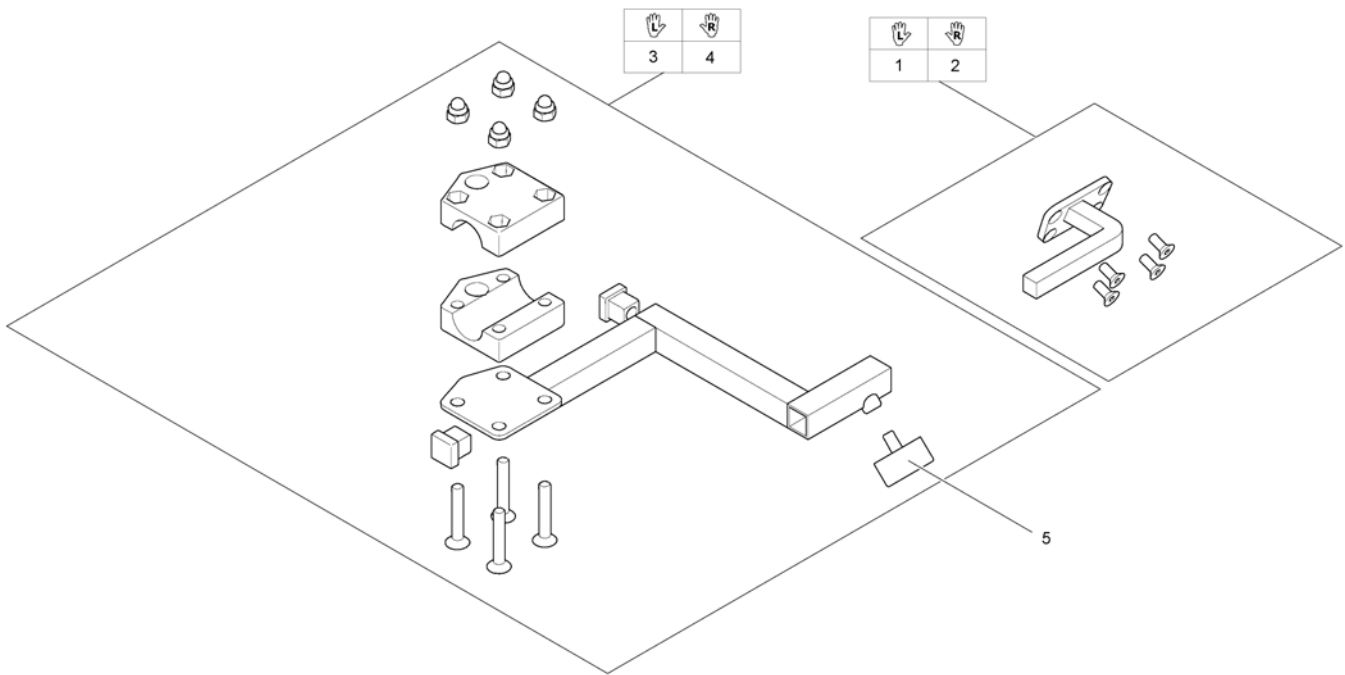

Pos.	Part number	Description	Valid from → until	Qty
1	SP1509435	Remote Support attendant control G90A, LH		1
2	SP1509439	Remote Support attendant control G90A, RH		1
3	SP1422937	Bracket for attendant control LH		1
4	SP1422936	Bracket for attendant control RH		1
5	L31001	Wing ScREW M6X10		1

Remote Bracket Attendant Ultra Low Maxx Seat


1588438 Rev

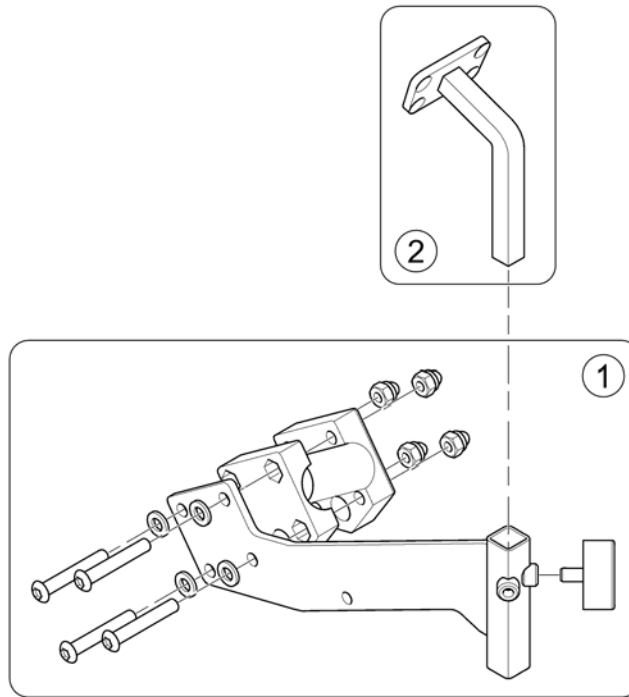

Pos.	Part number	Description	Valid from → until	Qty
1	SP1588427	Bracket Attendant Control		1
2	SP1587229	Adapter G90 (with screws)		1

Remote Bracket Attendant for Recaro-Seat



1563503.iso

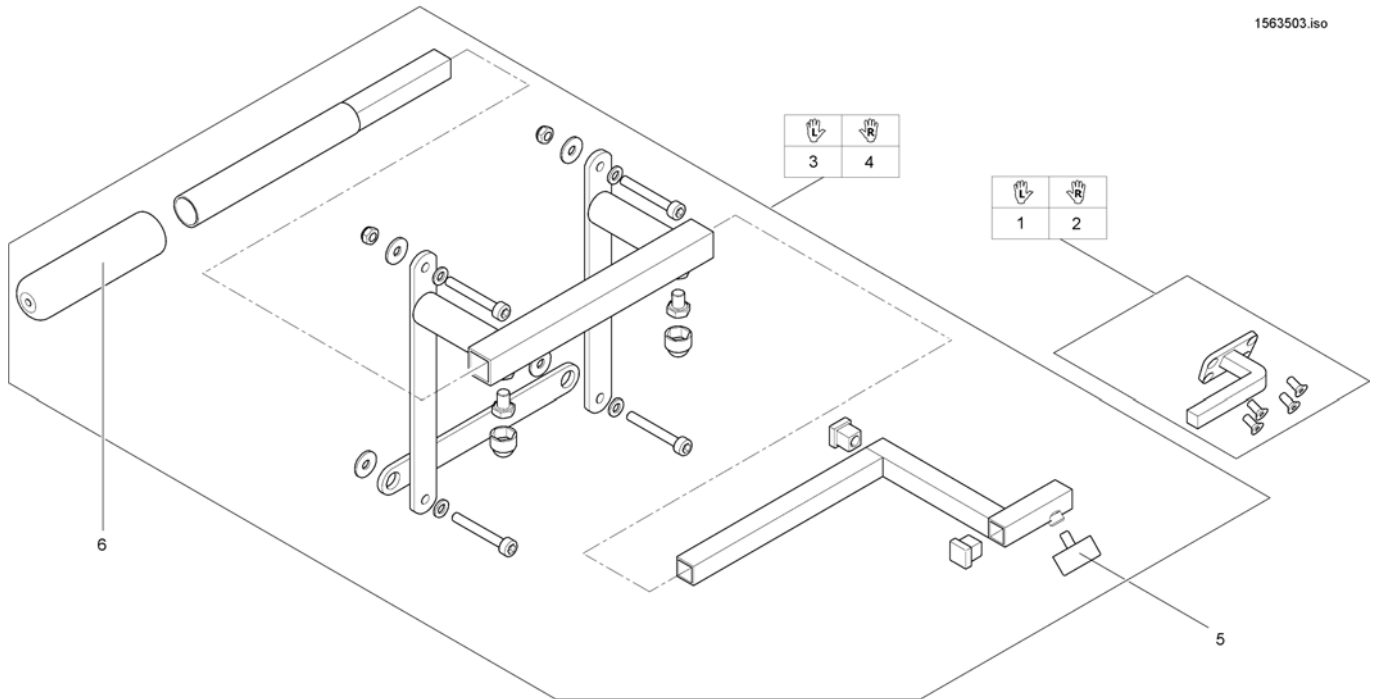

Pos.	Part number	Description	Valid from → until	Qty
1	SP1509435	Remote Support attendant control G90A, LH		1
2	SP1509439	Remote Support attendant control G90A, RH		1
3	SP1422814	Assebmly kit for dual control cpl., Recaro, LH		1
4	SP1422813	Assebmly kit for dual control cpl., Recaro, RH		1
5	L31001	Wing Scew M6X10		1
6	1435421	Handle plastic black Ø22 "Invacare"		1


G91S with brackets

1560680 iso

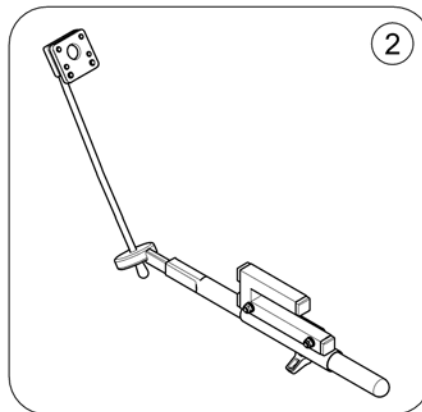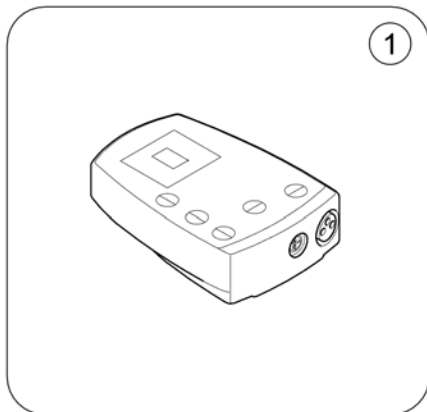

Pos.	Part number	Description	Valid from → until	Qty
1	SP1505187	Remote Control G91S <i>[1615]</i>		1
2	-	Remote Bracket Standard <i>[1615+1207/1208]</i>		1


Remote Bracket Standard

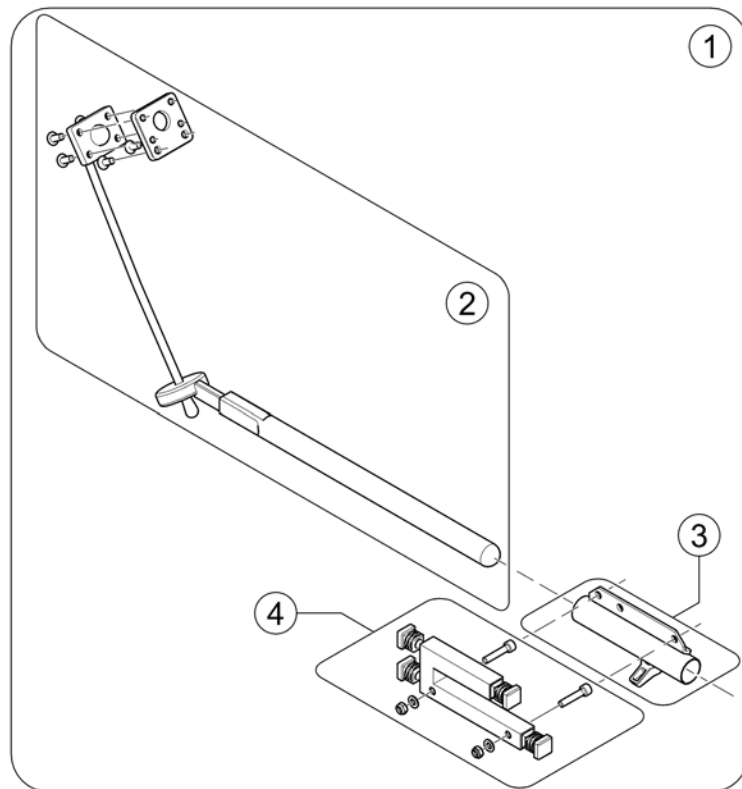

Pos.	Part number	Description	Valid from → until	Qty
1	SP1509690	Remote Bracket G91/G91S LH/RH, cpl.		1
2	SP1527504	Swan neck remote bracket with adapter plate		1
3	SP1603801	Mounting bracket, cpl. <i>Old bracket no longer available!</i>	2013-08-05 →	1
4	SP1603847	Mounting Adapter G91 <i>Old bracket no longer available!</i>	2013-08-05 →	1

Centre control REM 41

F383057.dwg


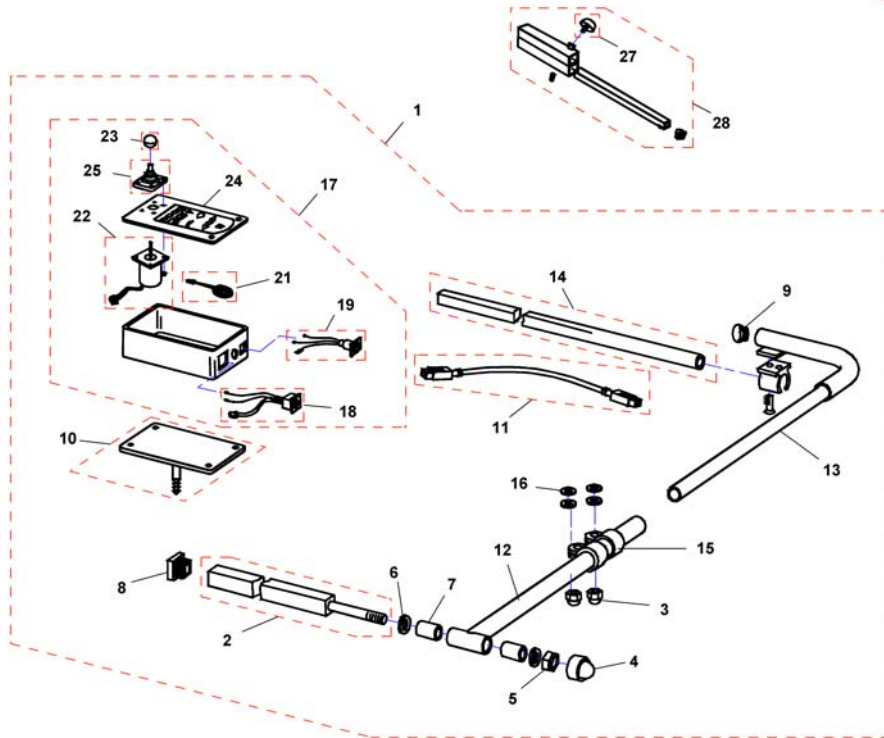


Pos.	Part number	Description	Valid from → until	Qty
-	Q99081	User Manual Table- & Middle Control REM41-S German		1
1	SP1530588	CENTRAL STEERING CPL., SWING-AWAY RH, REM41-D		1
1	SP1530589	CENTRAL STEERING CPL., SWING-AWAY LH, REM41-D		1
2	Z177024.00	SUPPORT TUBE; CHROME PLATED		1
3	F60208	Cap nut DIN1587-M08 zc		1
4	L01037	Sks Nut Cap M12		1
5	F20106	Hexagon nut DIN985-M12 zc		1
6	G00104	Washer DIN125-Ø13 zc		1
7	Z17032A	Bush Rd. 19 X 3.5 X 25		1
8	L01055	Plug 15x15mm		1
9	L01008	Tube End Plug SI 22		1
10	Z183216.P40	SUPPORT BRACKET; RAL 9005 BLACK		1
11	1559862	Bus cable ACS, 1700mm		1
12	Z183089.00	SWING ARM; CHROME PLATED		1
13	Z183086.00	SUPPORT ARM RH; CHROME PLATED		1
13	Z183092.00	SUPPORT ARM LH; CHROME PLATED		1
14	H281005A.00	Support rail, chrome plated		1
15	5006720	Klemmschelle F. Dm 22 mm		1
16	G00106	Washer DIN125-A-Ø8,4 zc		1
17	SP1527930	DX ACS Tray Remote DX-REM41		1
18	P41108	ACS-BUS CABLE SOCKET		1
19	P41106	CHARGER SOCKET		1
21	P30493	HORN		1
22	GSM55120	ACS-remote WITH 180MM CABLE		1
23	L36999	Joystick Knopf Standard Ball		1
24	-	No longer available.		0
25	Z183001A	Rubber gaiter for ACS remote		1
26	Z183289	CENTRAL STEERING SUPPORT TUBE CPL.		1
27	L31001	Wing Scew M6X10		1
28	Z183289	CENTRAL STEERING SUPPORT TUBE CPL.		1

Chin Control, small (=> 16/12/31)

Pos.	Part number	Description	Valid from → until	Qty
-		Small Chincontrol		1
-		Framework for chincontrol		1


Small Chincontrol

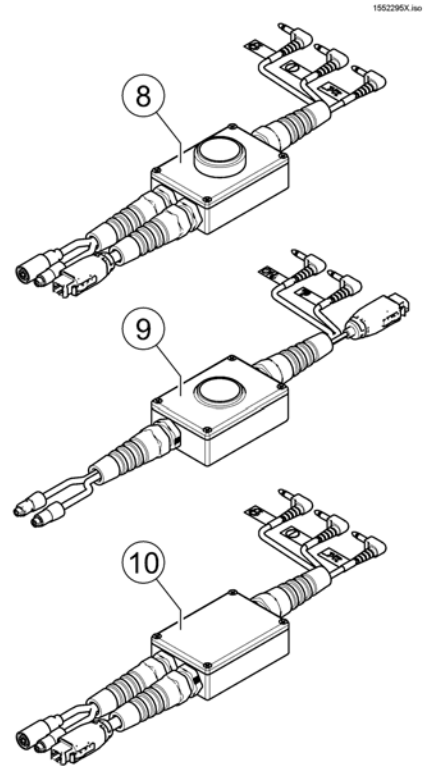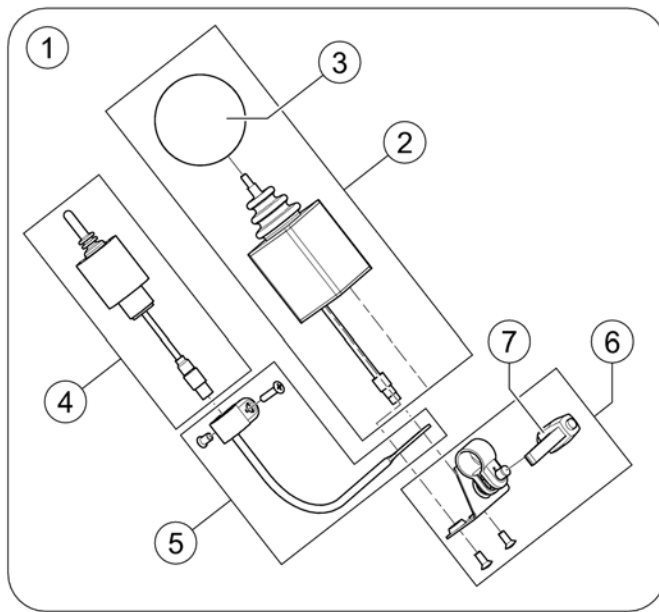

1552295X.iso

Pos.	Part number	Description	Valid from → until	Qty
1	1448705	Chin control REM24, cpl.		1
1	1421770	Chin control G80/G90, cpl.		1
1	SP1544255	Chin control G91, cpl.		1
2	Z183282	Joystick, DX-RJM (with Binder plug, 4-pole)		1
3	Z183265	Joystick Knob Sponge Ball		1
4	1442943	Switchjoystick for REM 24 SD, cpl.		1
4	Z183283	Switchjoystick G80/G90, cpl.		1
4	SP1544250	Switchjoystick for G91S, cpl.		1
5	SP1552294	Switchjoystick bracket		1
6	SP1552295	Joystick clamp		1
7	L32901	Clamping lever		1
8	SP1442944	Harness Chin Control for REM 24 SD		1
8	1442963	Harness Chin Control for REM 24 SD (long version) vertical swing away		1
9	Z183275	Harness Chin Control for G80/G90		1
10	SP1541715	Harness Chin Control for G91S		1



Framework for chincontrol

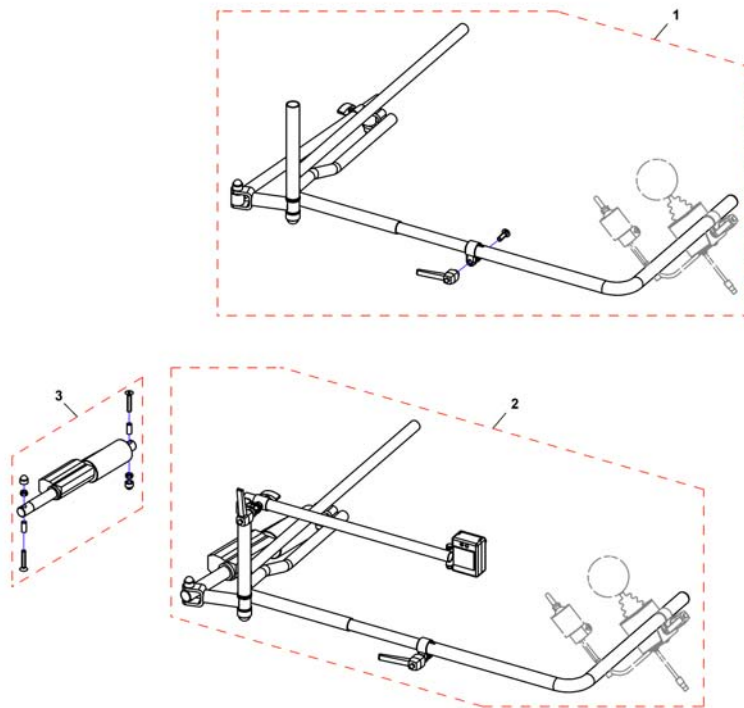



Pos.	Part number	Description	Valid from → until	Qty
-	Z167994	CHIN CONTROL SUPPORT . CPL. <i>Not shown / Nicht abgebildet / Non illustré</i>		1
1	SP1422943	Framework for chin control, manual, horizontal swing away, LH, cpl.		1
1	SP1422942	Framework for chin control, manual, horizontal swing away, RH, cpl.		1
2	SP1422945	Framework for chincontrol, powered, horizontal swing away, LH, cpl. <i>incl. Actuator</i>		1
2	SP1422944	Framework for chin control, powered, horizontal swing away, RH, cpl. <i>incl. Actuator</i>		1
-	SP1552335	Hardware, chin control, powered, horizontal swing away, LH <i>Not shown / Nicht abgebildet / Non illustré</i>		1
-	SP1552336	Hardware, chin control, powered, horizontal swing away, RH <i>Not shown / Nicht abgebildet / Non illustré</i>		1
-	SP1422946	Framework for chincontrol, powered, vertikal swing away <i>Not shown / Nicht abgebildet / Non illustré</i>		1
3	SP1552337	Actuator for horizontal swing away chin control, cpl.		1
3	SP1552418	Gestell für Kinnbedienung, elektrisch, vertikal abschwenkbar <i>Not shown / Nicht abgebildet / Non illustré</i>		1

Chin Control - Modulite Seat (15/10/01 =>)

1591326.iss

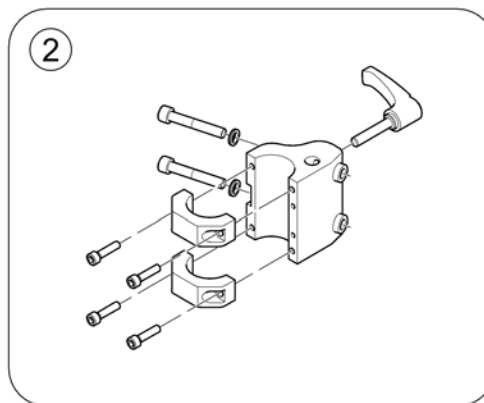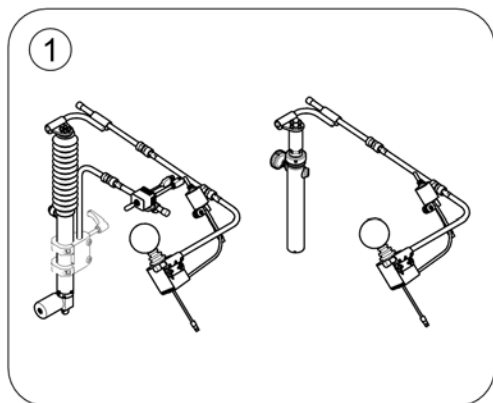


Pos.	Part number	Description	Valid from → until	Qty
-	-	If retrofitting, the user manual must be ordered separately!		1
-	-	User Manuals Chin Control (2015-10-01 =>)		1
1	-	Chin Control DX-RJM		1
2	-	Mounting Hardware - Chin Control		1

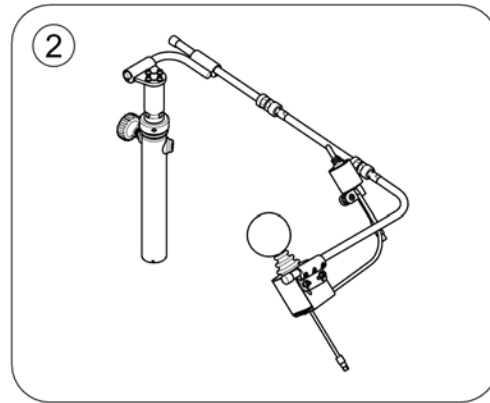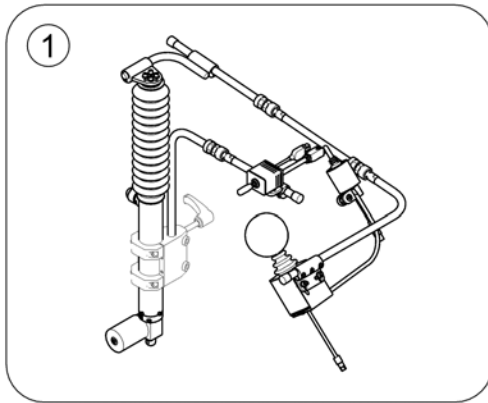

Pos.	Part number	Description	Valid from → until	Qty
1	-	Chin Control powered - Details		1
2	-	Chin Control manual - Details		1
-	SP1601389	Kit, Swing-away mounting arm, powered, LH, cpl. with stereo-switch (no joystick nor harness included) <i>Please order respective harness & joystick separately!</i>		1
-	SP1601390	Kit, Swing-away mounting arm, powered, RH, cpl. with stereo-switch (no joystick nor harness included) <i>Please order respective harness & joystick separately!</i>		1
-	SP1601391	Kit, Swing-away mounting arm, manual, LH, cpl. (no joystick nor harness included) <i>Please order respective harness & joystick separately!</i>		1
-	SP1601408	Kit, Swing-away mounting arm, manual, RH, cpl. (no joystick nor harness included) <i>Please order respective harness & joystick separately!</i>		1
-	SP1601528	Kit, Joysticks & Harness for REM 24 SD		1
-	SP1601529	Kit, Joysticks & Harness for G91S		1

Chin Control powered - Details


1591297.jpg

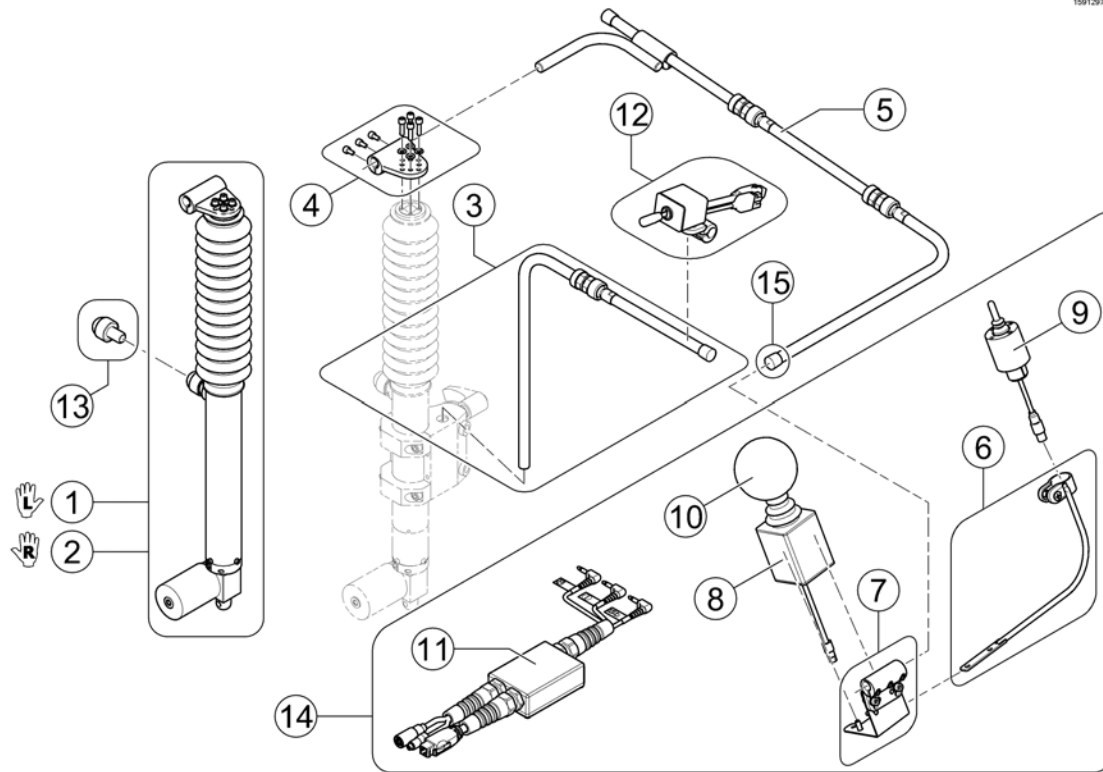

Pos.	Part number	Description	Valid from → until	Qty
-	SP1605661	Fixation Kit (cable ties, spiral coiled tube & end caps)	2016-04-01 →	1
1	SP1580094	Actuator cpl. LH		1
2	SP1591292	Actuator cpl. RH		1
3	SP1605659	Remote Rod Assy <i>New designed bracket (2016-04-01 =>) for Modulite Seat needed!</i>	2016-04-01 →	1
4	SP1589144	Rod cylinder connection chin, cpl.		1
5	SP1605660	Joystick Rod Assy	2016-04-01 →	1
6	SP1589146	Mounting arm for Switch Joystick		1
7	SP1589142	Mounting hardware for DX-RJM, cpl.		1
8	Z183282	Joystick, DX-RJM (with Binder plug, 4-pole)		1
9	1442943	Switchjoystick for REM 24 SD, cpl.		1
9	SP1544250	Switchjoystick for G91S, cpl.		1
10	Z183265	Joystick Knob Sponge Ball		1
11	SP1442944	Harness Chin Control for REM 24 SD		1
11	SP1541715	Harness Chin Control for G91S		1
12	SP1580701	Stereo-switch box, cpl. with mounting hardware		1
13	SP1589145	Adjustment Screw		1
14	SP1601528	Kit, Joysticks & Harness for REM 24 SD		1
14	SP1601529	Kit, Joysticks & Harness for G91S		1
15	SP1602721	End cap A12, black	2016-04-01 →	3

Chin Control manual - Details


1591296/iso

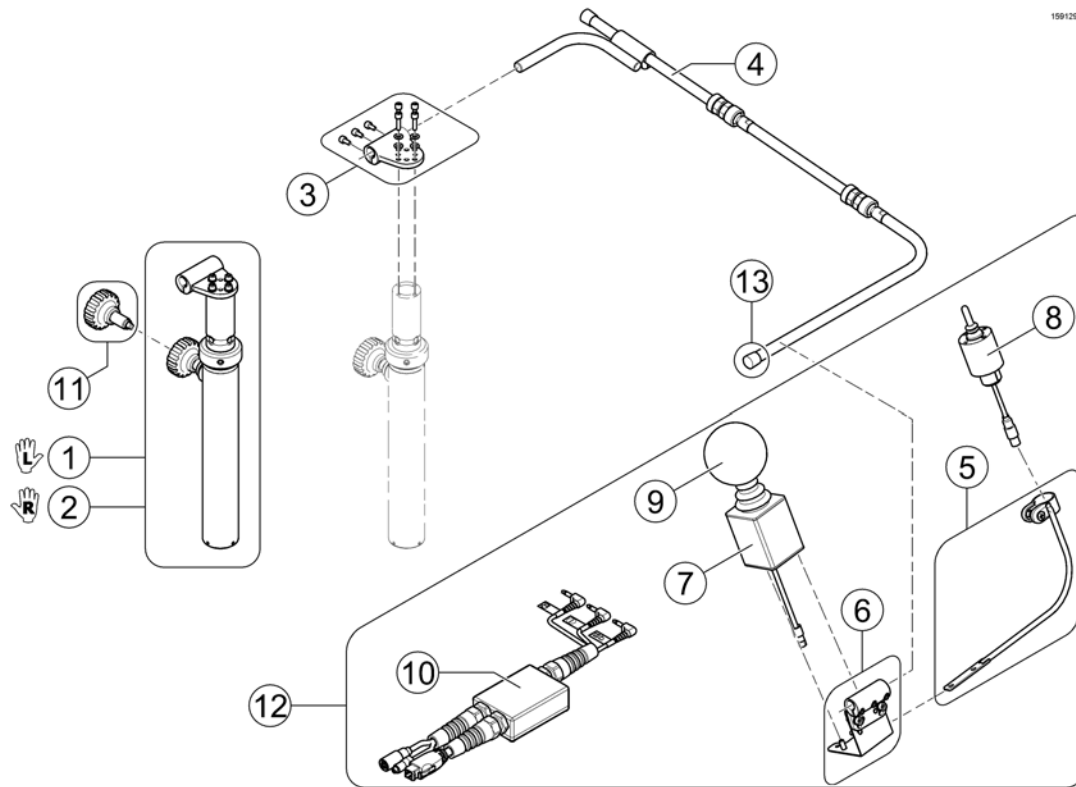

Pos.	Part number	Description	Valid from → until	Qty
-	SP1605661	Fixation Kit (cable ties, spiral coiled tube & end caps)	2016-04-01 →	1
1	SP1577456	ASM Chin Control man. LH, cpl.		1
2	SP1591293	ASM Chin Control man. RH, cpl.		1
3	SP1589144	Rod cylinder connection chin, cpl.		1
4	SP1605660	Joystick Rod Assy	2016-04-01 →	1
5	SP1589146	Mounting arm for Switch Joystick		1
6	SP1589142	Mounting hardware for DX-RJM, cpl.		1
7	Z183282	Joystick, DX-RJM (with Binder plug, 4-pole)		1
8	1442943	Switchjoystick for REM 24 SD, cpl.		1
8	SP1544250	Switchjoystick for G91S, cpl.		1
9	Z183265	Joystick Knob Sponge Ball		1
10	SP1442944	Harness Chin Control for REM 24 SD		1
10	SP1541715	Harness Chin Control for G91S		1
11	SP1580666	Adjustment Handle		1
12	SP1601528	Kit, Joysticks & Harness for REM 24 SD		1
12	SP1601529	Kit, Joysticks & Harness for G91S		1
13	SP1602721	End cap A12, black	2016-04-01 →	2

Mounting Hardware - Chin Control


1591309 iso

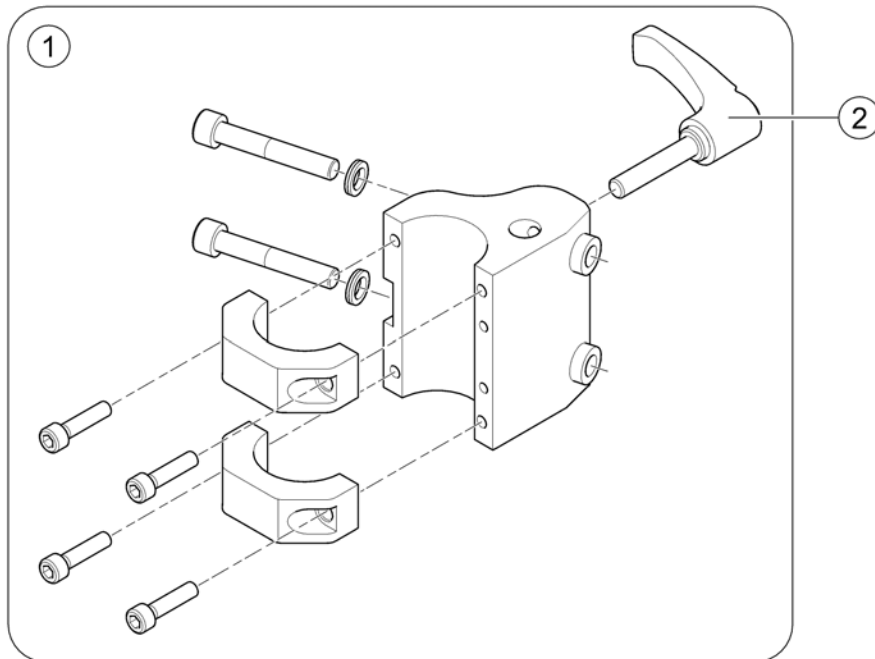

Pos.	Part number	Description	Valid from → until	Qty
1	SP1577173	Mounting Hardware, Chin Control - Modulite Seat <i>Design change 2016-04-01 =></i>		1
2	SP1604355	Clamp Lever M8x40		1

Dual Control with brackets


1564180 00

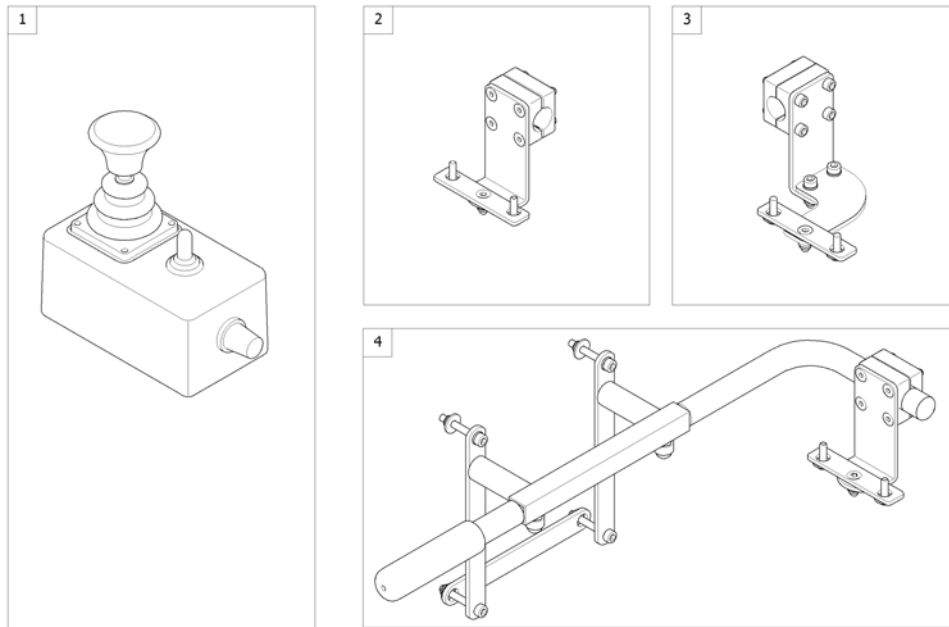

Pos.	Part number	Description	Valid from → until	Qty
1	6001425	ACS dual control [1291/1292]		1
2	-	Remote Bracket Dual Control for Standard-/EasyAdapt-Seat [1291/1292]		1
3	-	Remote Bracket Dual Control for Modulite Seat		1
4	-	Remote Bracket Dual Control for Recaro Seat [1291/1292]		1

Remote Bracket Dual Control for Standard-/EasyAdapt-Seat


1560885.iso

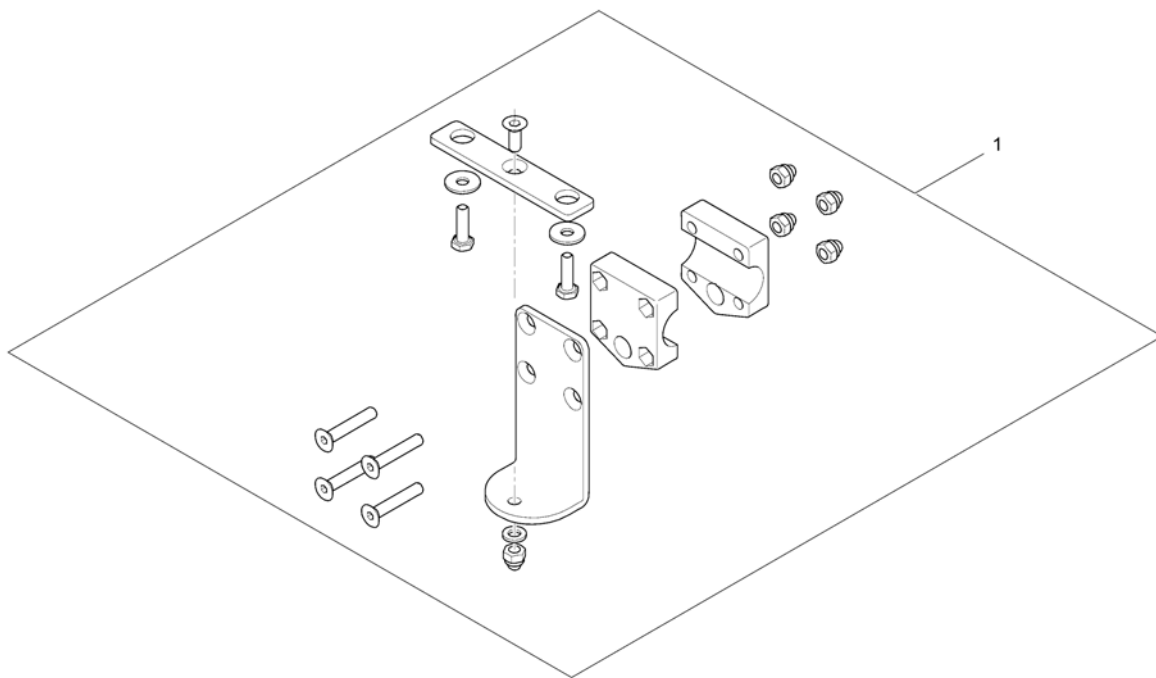

Pos.	Part number	Description	Valid from → until	Qty
1	SP1422818	Remote Bracket Dual Control		1

Remote Bracket Dual Control for Modulite Seat


1574291 iso

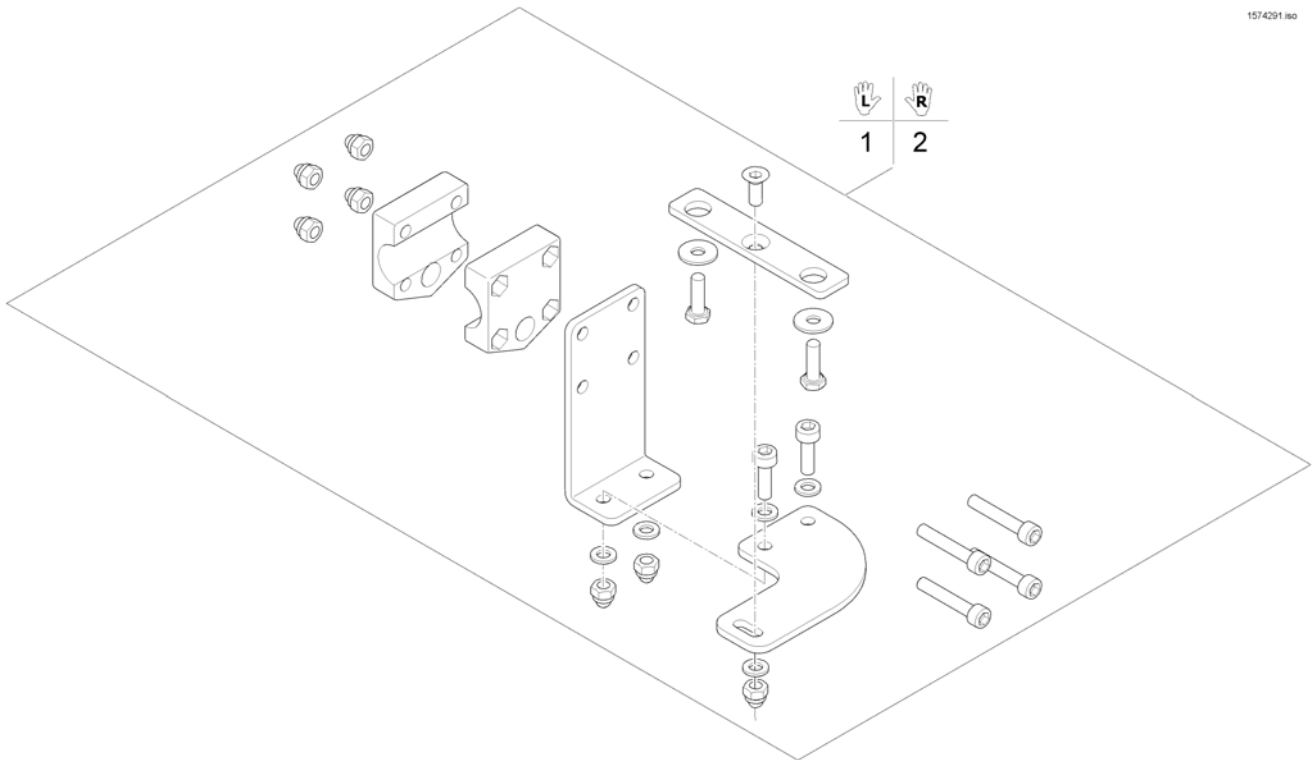

Pos.	Part number	Description	Valid from → until	Qty
1	SP1571799	Remote Bracket Dual Control LH		1
2	SP1571800	Remote Bracket Dual Control RH		1

Remote Bracket Dual Control for Recaro Seat


1563504.iso

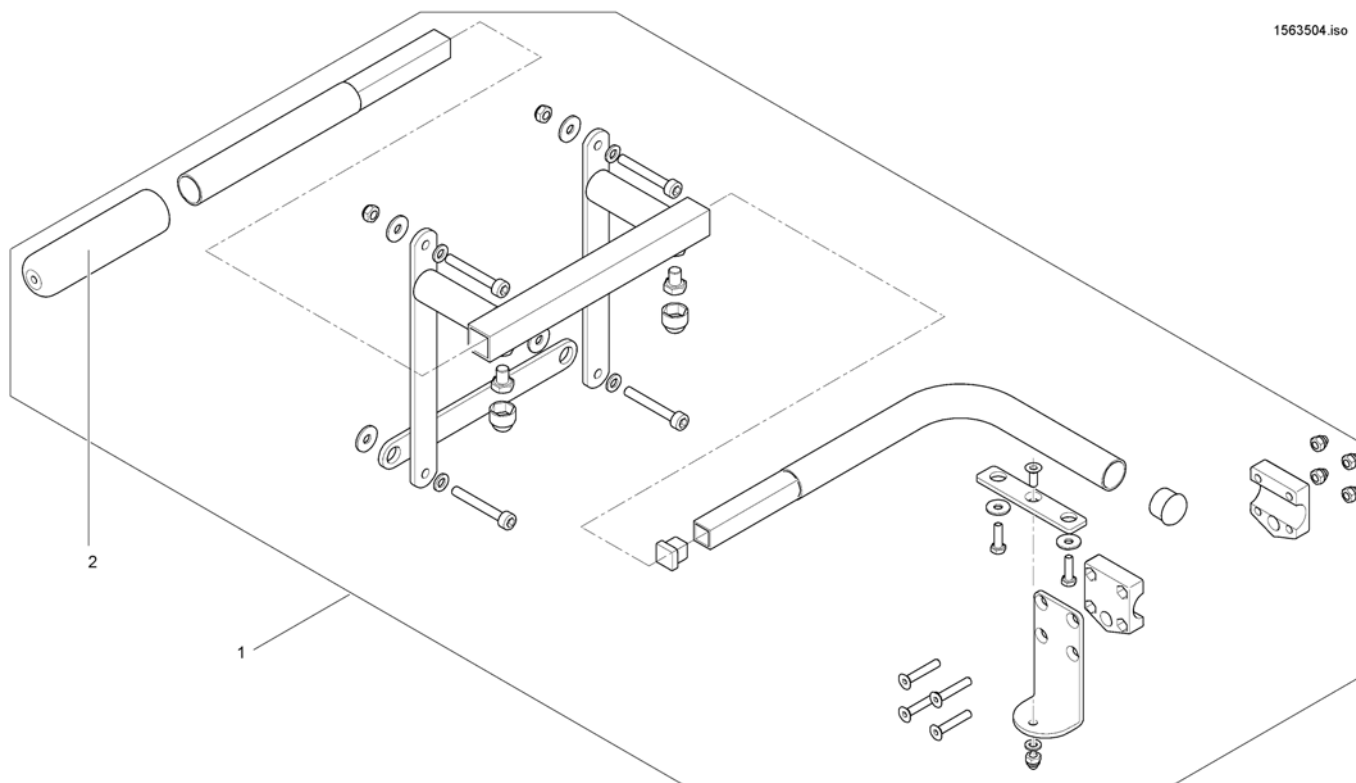

Pos.	Part number	Description	Valid from → until	Qty
1	SP1422819	Assembly kit for dual control Recaro		1
2	1435421	Handle plastic black Ø22 "Invacare"		1

Compact Dual Control with brackets


1588511.000

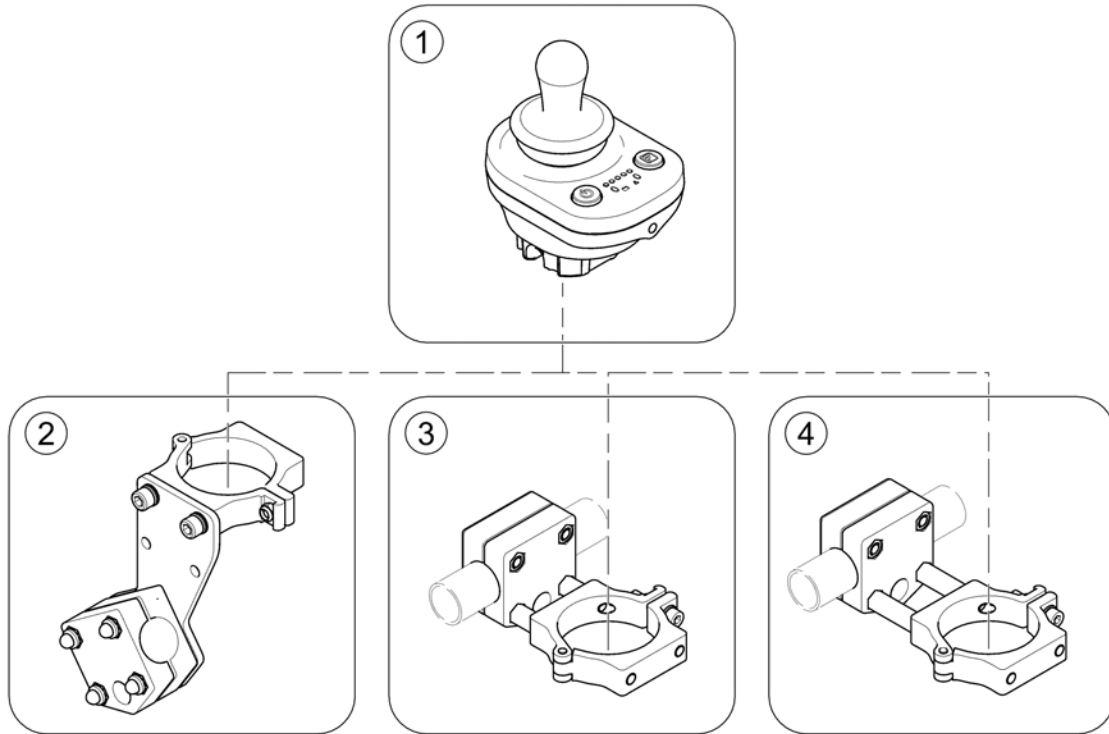

Pos.	Part number	Description	Valid from → until	Qty
1	-	Compact Dual Control [1720]		1
2	-	Bracket, Compact Dual Control - Ultra Low Maxx Seat [1207/1208 + 1720 for 2020]	2015-06-03 →	1
3	-	Bracket, Compact Dual Control - short	2015-07-01 →	1
4	-	Bracket, Compact Dual Control - long	2015-07-01 →	1

Compact Dual Control


1586988.jpg

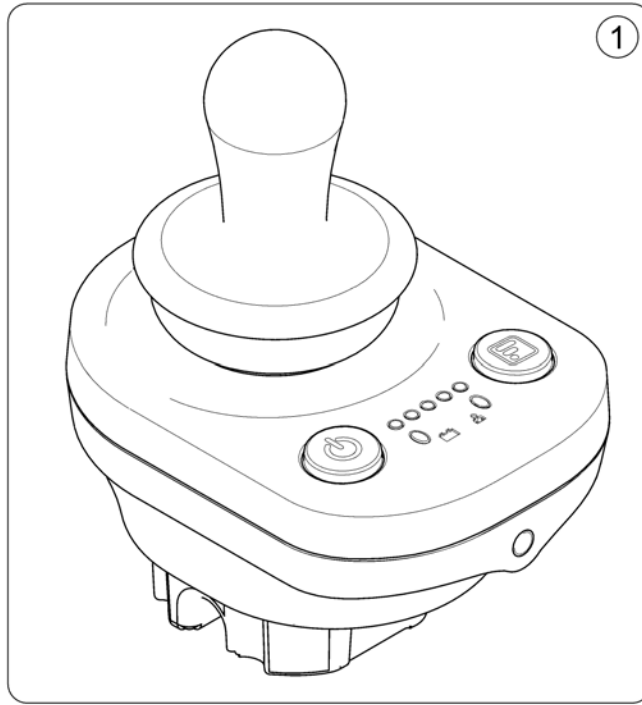

Pos.	Part number	Description	Valid from → until	Qty
1	SP1586988	Remote DX2-ACU		1

Bracket, Compact Dual Control - Ultra Low Maxx Seat


1586512.00

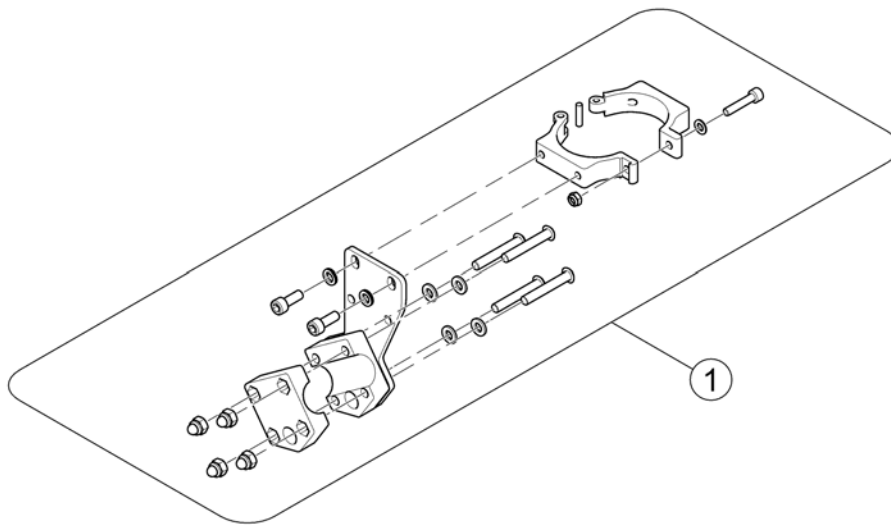

Pos.	Part number	Description	Valid from → until	Qty
1	SP1586795	Bracket, Compact Dual Control LH/RH		1

Bracket, Compact Dual Control - short


1588857.jpg

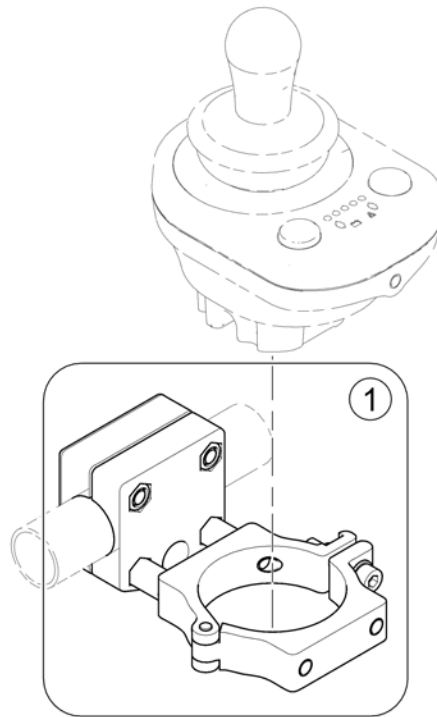

Pos.	Part number	Description	Valid from → until	Qty
1	SP1587904	Remote Attachment - short		1

Bracket, Compact Dual Control - long

158856 iso

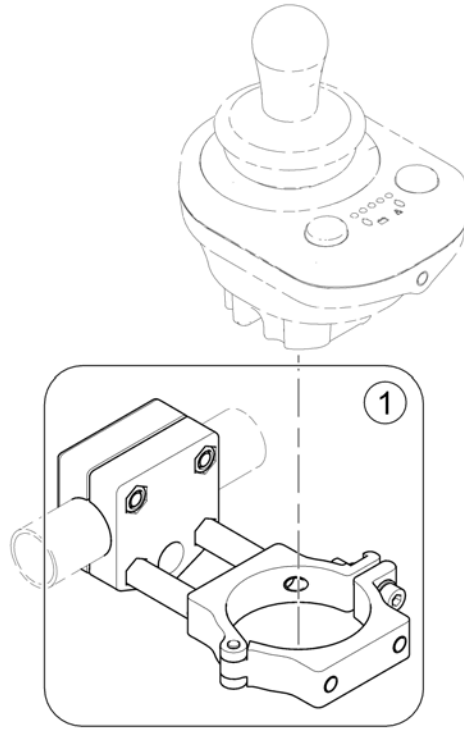


Pos.	Part number	Description	Valid from → until	Qty
1	SP1587119	Remote Attachment - long		1

Intuitive Dual Control (IDC) with brackets

Pos.	Part number	Description	Valid from → until	Qty
-		IDC with bracket		1
-		IDC with bracket (Ultra Low Maxx Seat)		1
-		Height adjustable bracket IDC		1

IDC with bracket


1602413.iss

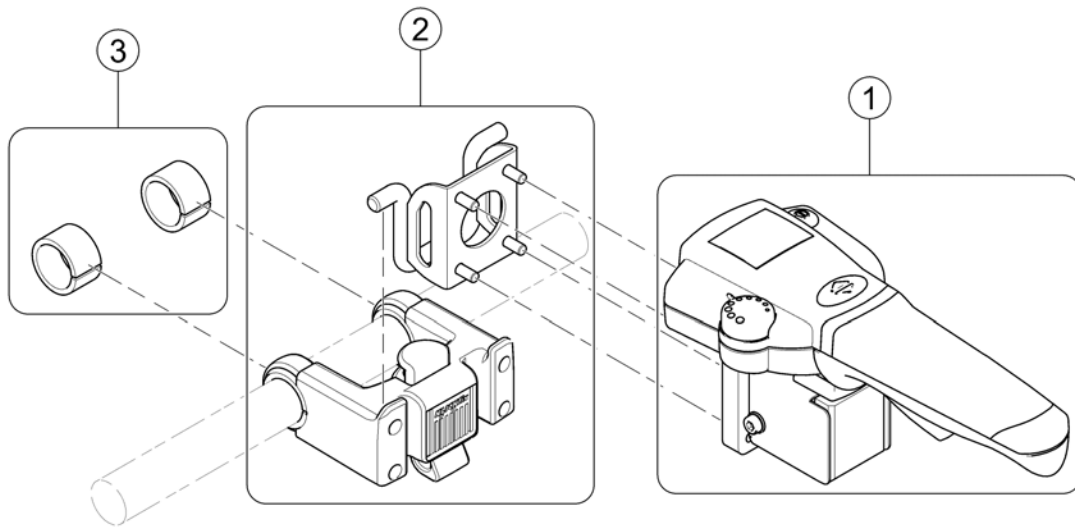

Pos.	Part number	Description	Valid from → until	Qty
-		The IDC incl. mounting hardware has been revised, the old hardware is no longer available.	2016-01-01 →	0
-		If retrofitting, the user manual must be ordered separately!		1
-		User Manuals IDC (Intuitive Dual Control)		1
1	SP1603425	Intuitive Dual Control (revised), cpl.		1
2	SP1603426	IDC Bracket		1
3	SP1601919	Rubber cuts		1

IDC with bracket (Ultra Low Maxx Seat)


1602412.iss

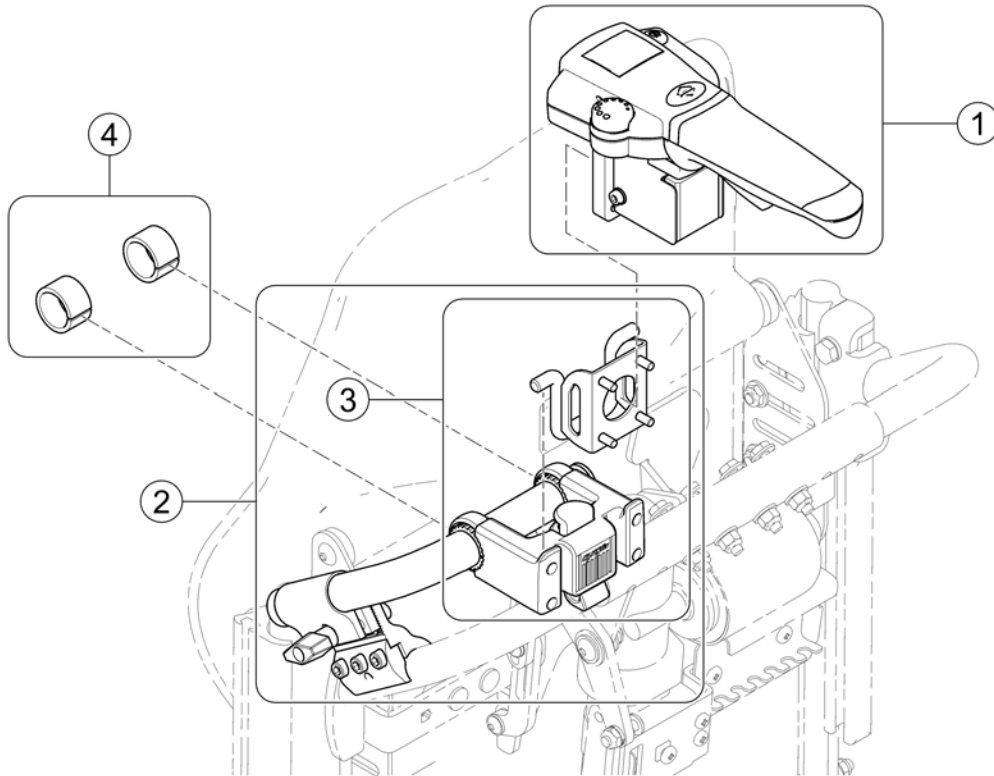

Pos.	Part number	Description	Valid from → until	Qty
-	-	The IDC incl. mounting hardware has been revised, the old hardware is no longer available.	2016-01-01 →	0
-	-	If retrofitting, the user manual must be ordered separately!		1
-	-	User Manuals IDC (Intuitive Dual Control)		1
1	SP1603425	Intuitive Dual Control (revised), cpl.		1
2	SP1603427	IDC Bracket, cpl. with mounting hardware for Ultra Low Maxx Seat		1
3	SP1603426	IDC Bracket		1
4	SP1601919	Rubber cuts		1

Height adjustable bracket IDC


1605484 iso

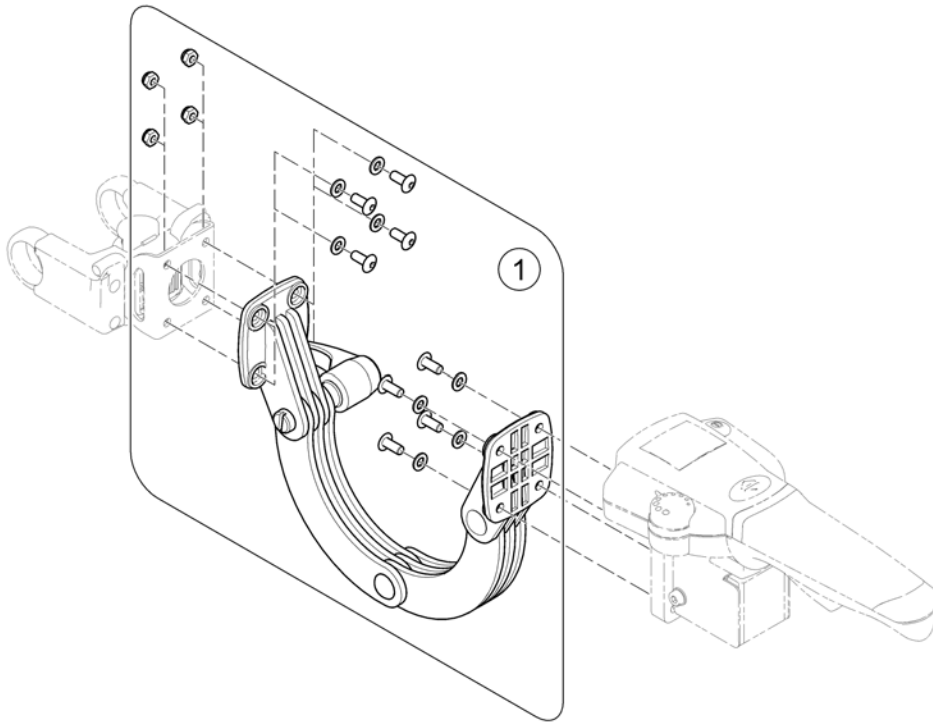

Pos.	Part number	Description	Valid from → until	Qty
1	SP1604195	Height adjustable bracket for IDC, cpl.		1

ACS Cables & Accessories

Pos.	Part number	Description	Valid from → until	Qty
-		Bus-Cables ACS		1
-		T-Adapter		1
-		T-Adapter (Ultra Low Maxx Seat)		1
-		Backrest Sensor - Ultra Low Maxx Seat		1
-		Tilt Sensor - Ultra Low Maxx Seat		1
-		Programming Kits		1
-		Magnetkey for Remotes		1

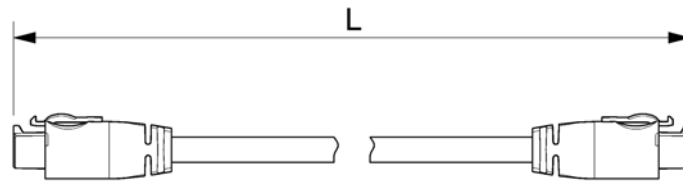


Position	L [mm]
1	300
2	500
3	700
4	1100
5	1700
6	2000
7	2500
8	2700
9	1060

Pos.	Part number	Description	Valid from → until	Qty
1	GSM63003	Bus cable ACS, 300mm		1
2	SPGSM63005	Bus cable ACS - 500mm <i>Replace GSM63004</i>		1
3	GSM63007	Bus cable ACS 700mm		1
4	GSM63011	Bus cable ACS, 1100mm		1
5	1559862	Bus cable ACS, 1700mm <i>Replace GSM63016</i>		1
6	GSM63020	Bus cable ACS, 2000mm <i>Replace 1419386</i>		1
7	1440424	Bus cable ACS, 2500mm		1
8	SP1559873	Bus cable ACS - 2700mm <i>Replace GSM63026</i>		1
9	SP1550402	Extension cable ACS - 1060mm		1

T-Adapter


1534069X.dwg

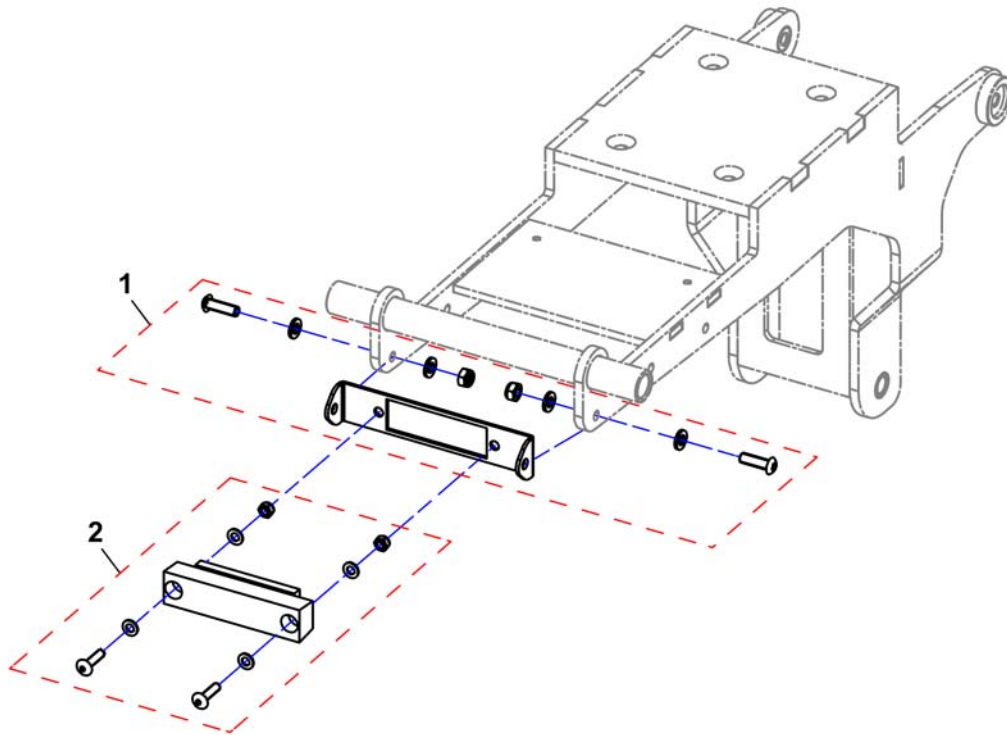

Pos.	Part number	Description	Valid from → until	Qty
1	SP1559812	Holder T-Adapter ACS	2012-04-20 →	1
1	SP1537966	Holder R-Net ACS	→ 2012-04-19	1
2	SP1537967	T-Adapter ACS		1

T-Adapter (Ultra Low Maxx Seat)


1586798 Rev

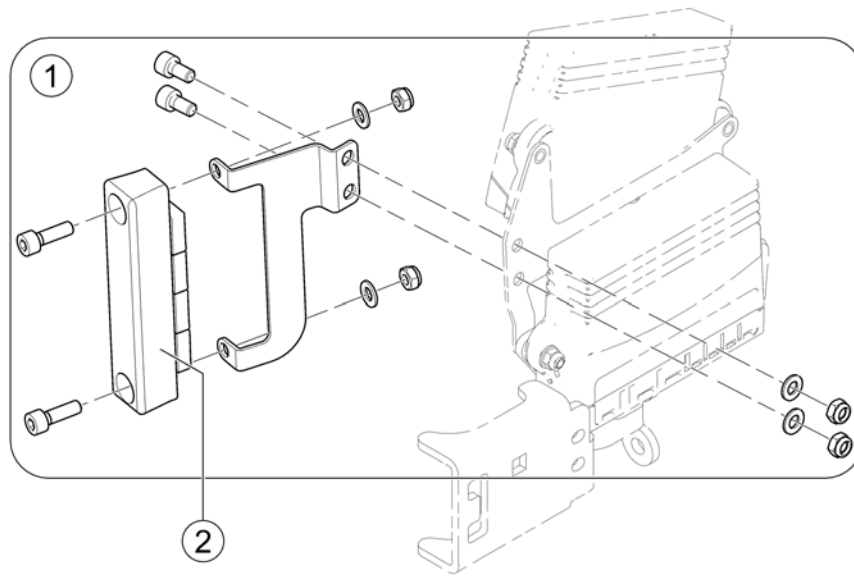

Pos.	Part number	Description	Valid from → until	Qty
1	SP1586798	T-Adapter with mounting hardware		1
2	P10072	Bus 4-way socket (T-Adapter)		1

Backrest Sensor - Ultra Low Maxx Seat


1602018.kim

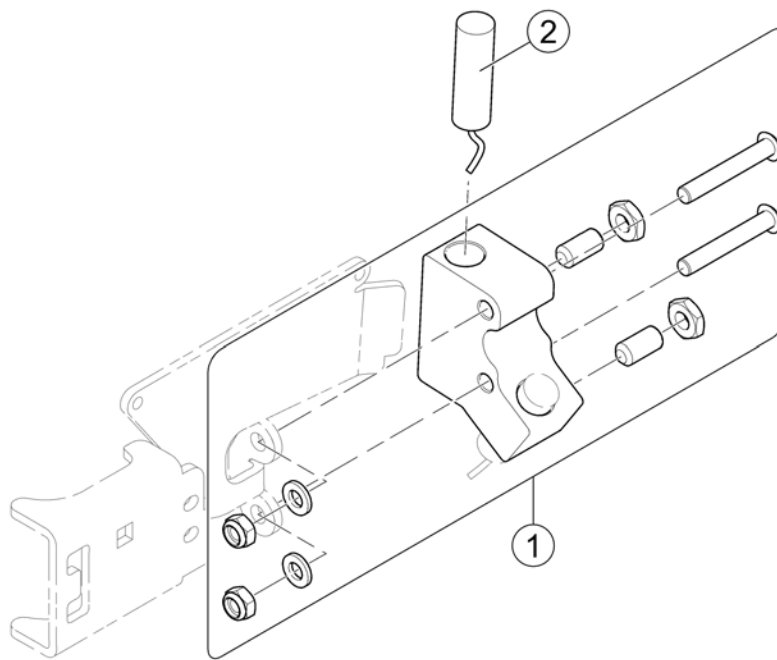

Pos.	Part number	Description	Valid from → until	Qty
1	SP1589753	Switch bracket		1
2	SPTRE0753	Tilt Switch		2

Tilt Sensor - Ultra Low Maxx Seat


1602019.kim

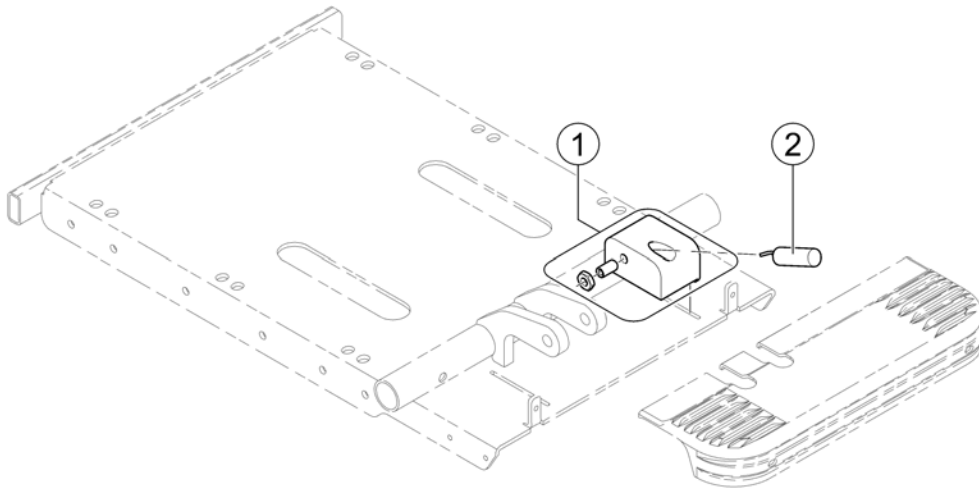


Pos.	Part number	Description	Valid from → until	Qty
1	SP1602017	Switch bracket		1
2	SPTRE0753	Tilt Switch		1

Programming Kits

1564805 iso

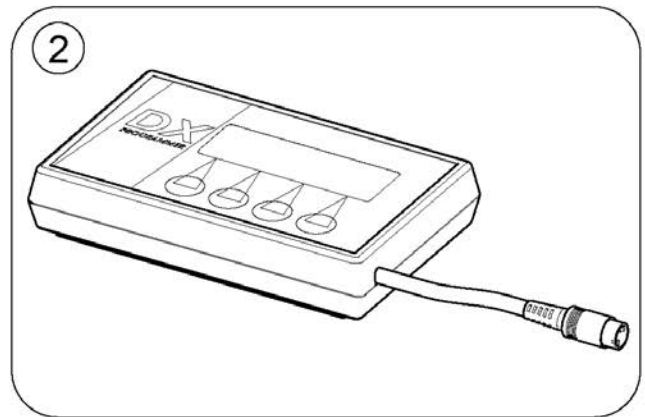

Pos.	Part number	Description	Valid from → until	Qty
1	1449280	USW wizard dongle kit, EDL-USW, cpl.		1
2	6001450	Hand Programmer, Instrucions In French, German, Spanish		1
	SP1441551	Programming Adapter Shark DK-ADAPT (necessary to program Shark)		1

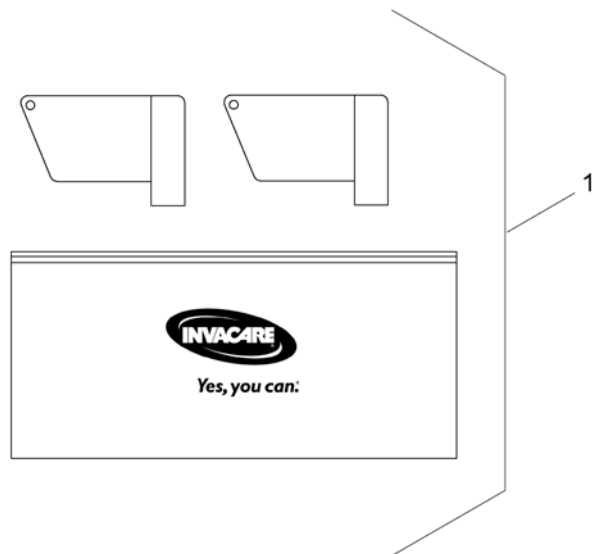


Pos.	Part number	Description	Valid from → until	Qty
1	SP1419695	Magnet key with bag		1

ACS Components

Pos.	Part number	Description	Valid from → until	Qty
-		iPortal Modules and Accessories		1
-		Remote, Joystick, Control		1
-		Joystick Knob		1
-		Buddy Button		1
-		Piko Button		1
-		Interface, Adapter, Other		1

iPortal Modules and Accessories


1603435 iso

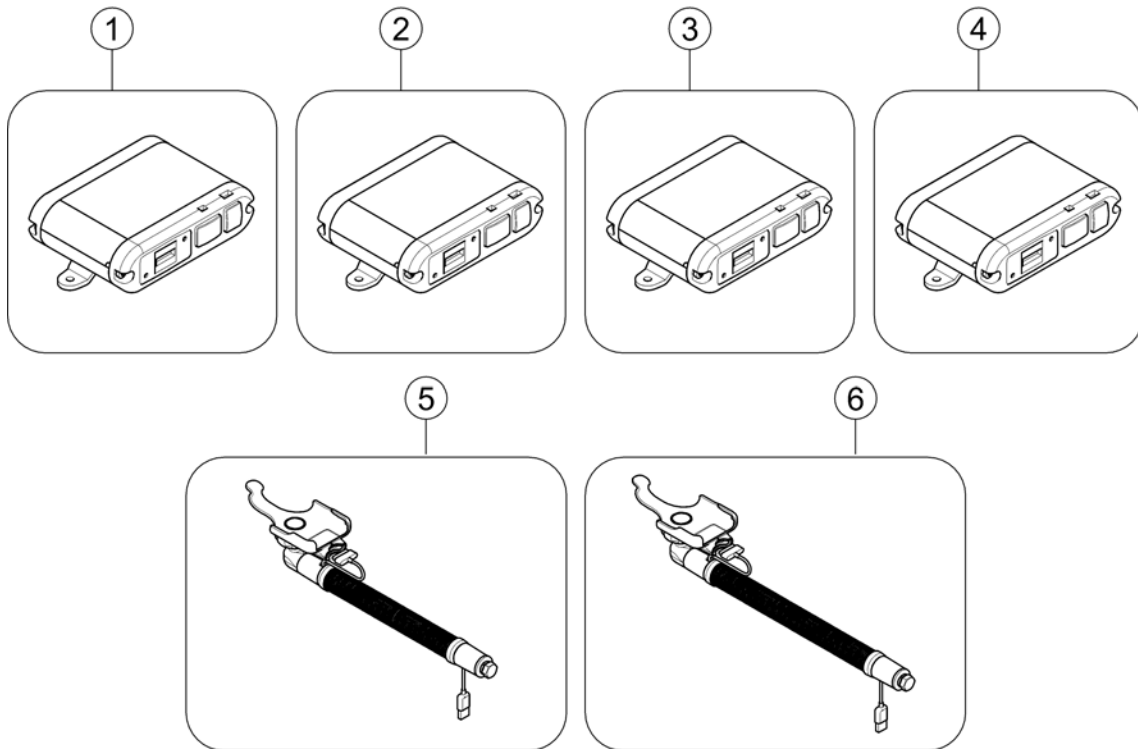


Pos.	Part number	Description	Valid from → until	Qty
1	SP1553373	iPortal Bluetooth Module Accessibility [1629]		1
2	SP1570392	iPortal Bluetooth Module Mouse Mover [1624]		1
3	SP1570393	iPortal Bluetooth Module Mouse Mover & Accessibility [1662]		1
4	SP1545530	iPortal Module without Accessibility [1634]		1
5	SP1545531	Gooseneck short for iPod touch & iPhone [1635]		1
6	SP1545532	Gooseneck long for iPod touch & iPhone [1636]		1

Remote, Joystick, Control

1603436.iss

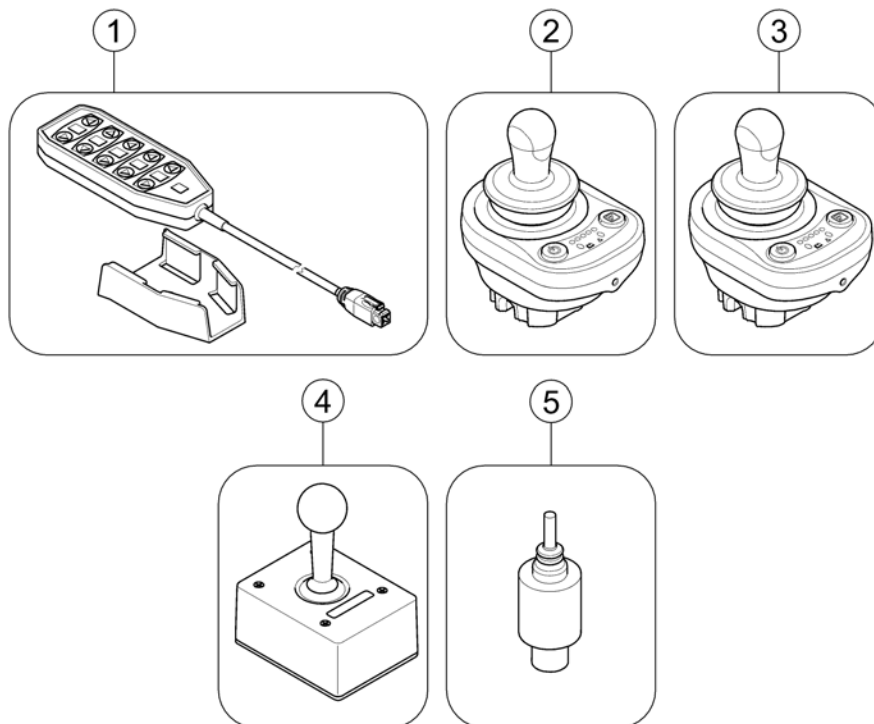


Pos.	Part number	Description	Valid from → until	Qty
1	P10079	Pendant control ARC 5 <i>[1641/1642]</i>		1
2	SP1586989	Remote Joystick <i>[1621]</i>		1
3	SP1586990	Remote Low-force Joystick <i>[1722]</i>		1
4	P10171	Heavy Duty Switch Joystick <i>[1620]</i>		1
5	P10175	Mini-Joystick with Push function <i>[1622]</i>		1
-	SP1442679	Head-Remote <i>[1613]</i>		1
-	SP1603442	Cable for Head-Remote ultrasonic proximity sensor		1

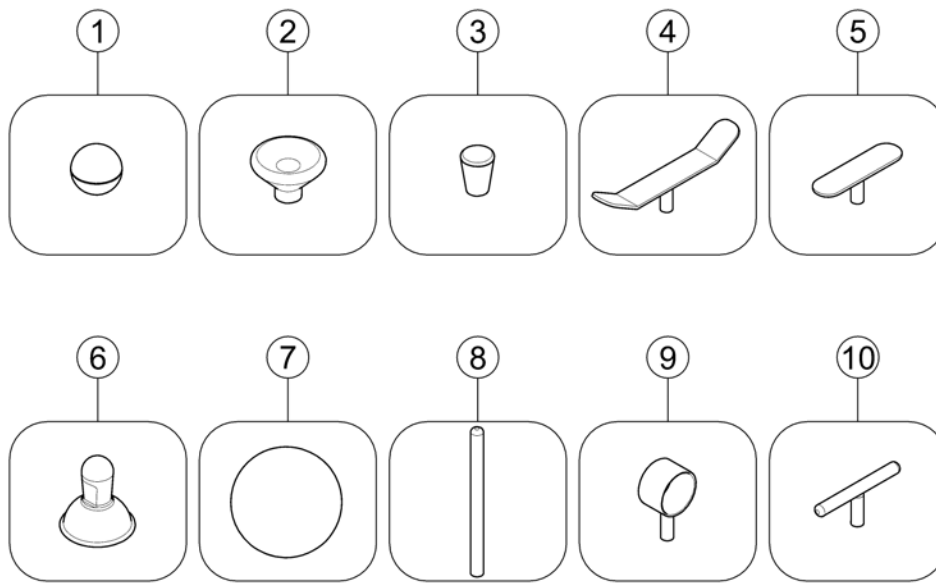

Pos.	Part number	Description	Valid from → until	Qty
1	L36999	Joystick Knopf Standard Ball [1650]		1
2	P10397	Joystick Knob Chin Cup [1651]		1
3	P39302	Joystick Knob Carrot [1652]		1
4	P39303	Joystick Knopf Flat T-Plate [1653]		1
5	P39305	Joystick Knob Flat T-Plate [1654]		1
6	P10306	Joystick Knopf DL [1655]		1
7	Z183265	Joystick Knob Sponge Ball [1656]		1
8	P39306	Joystick Knob Straight Bar [1657]		1
9	P39307	Joystick Knob Finger Tube [1658]		1
10	P39304	Joystick Knob Rount T-Bar [1659]		1

Buddy Button


1603438.ind

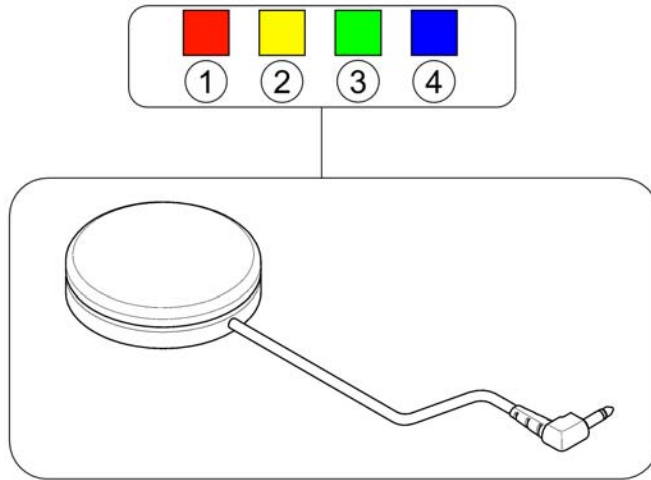

Pos.	Part number	Description	Valid from → until	Qty
1	5813964	Buddy Button red <i>[1685]</i>		1
2	P30093	Buddy Button yellow <i>[1686]</i>		1
3	P30092	Buddy Button green <i>[1687]</i>		1
4	P30089	Buddy Button blue <i>[1690]</i>		1

Piko Button


1603436.ind

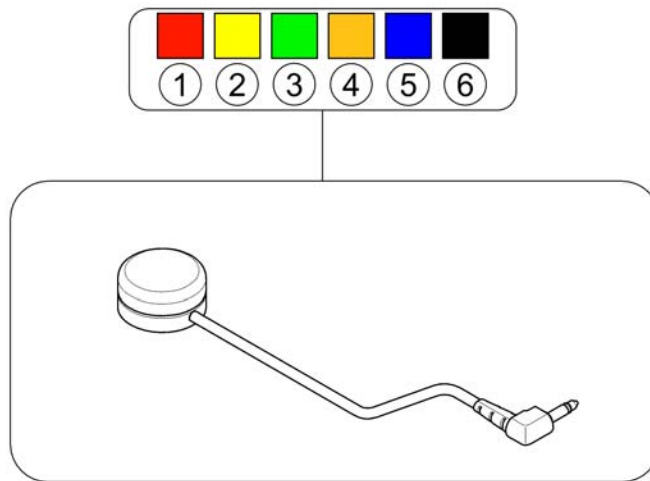

Pos.	Part number	Description	Valid from → until	Qty
1	SP1535717	Piko Button red [1711]		1
2	SP1535718	Piko Button yellow [1712]		1
3	SP1535723	Piko Button Green [1713]		1
4	SP1535724	Piko Button orange [1714]		1
5	SP1535725	Piko Button blue [1715]		1
6	SP1535726	Piko Button black [1716]		1





Interface, Adapter, Other

160340.iss

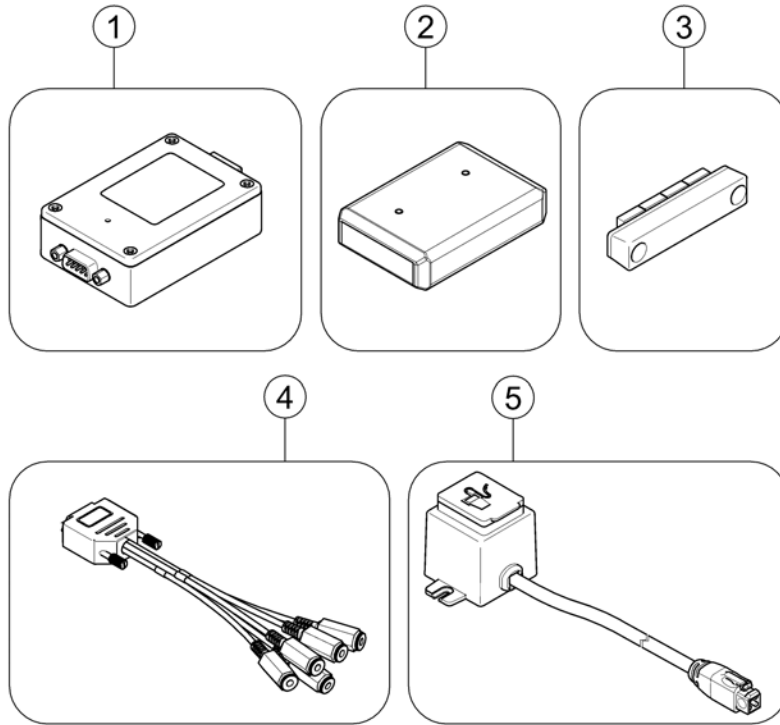

Pos.	Part number	Description	Valid from → until	Qty
1	SP1544036	4 Switch Interface [1630]		1
2	P10163	Sip & Puff Module(DX-SNP) [1633]		1
-	Z183267	Bracket for Sipp and Puff [1668]		1
3	P10072	Bus 4-way socket (T-Adapter) [1680]		1
4	P30080	5-1 Switch Adaptor [1681]		1
5	P40107	Charger Socket Modul [1661]		1

1602714.00

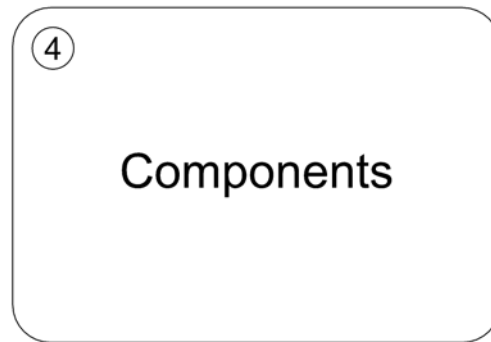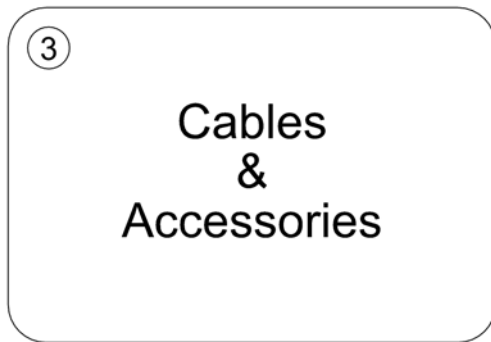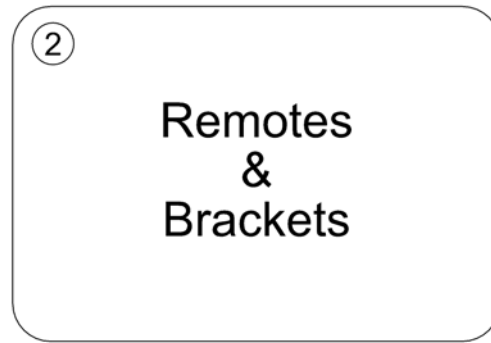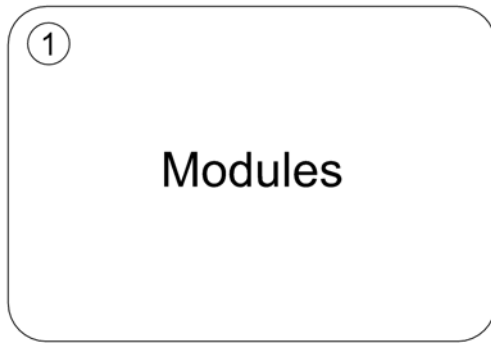


Pos.	Part number	Description	Valid from → until	Qty
1	-	Modules		1
2	-	Remotes & Brackets		1
3	-	Cables & Accessories		1
4	-	Components		1

Modules

Pos.	Part number	Description	Valid from → until	Qty
1	-	Power Modules		1
2	-	Bracket, Power Module		1
3	-	Actuator Modules (ACTs)		1
4	-	G-Trac Sensor / Gyro		1

Power Modules

1639232_00

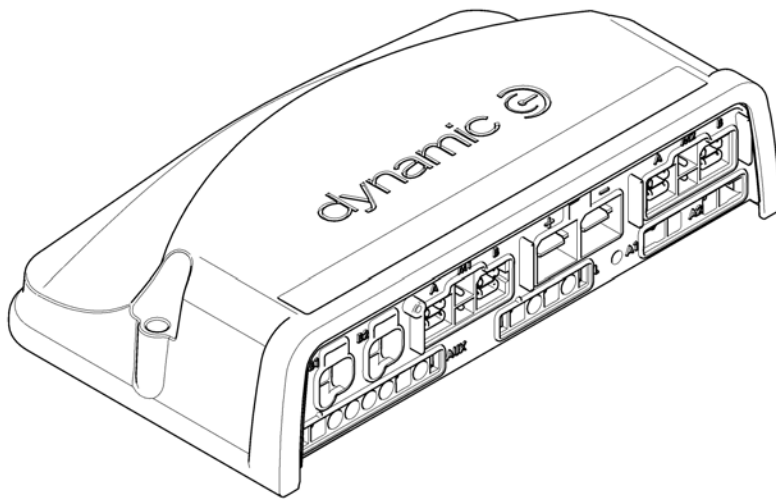

Storm4 Profiles

- 1
 R05G06A0A00X-00
 R05G06A0A00X-00
 R05G10A0A00X-00


- 2
 R05H06A1A00L-00
 R05H06A6A00L-00
 R05H06A7A00L-00
 R05H06A8A00L-00
 R05H06A1A00T-00
 R05H06A6A00T-00
 R05H06A7A00T-00
 R05H06A8A00T-00

- 3
 R01H10A1A00L-00
 R01H10A6A00L-00
 R01H10A7A00L-00
 R01H10A8A00L-00
 R01H10A1A00T-00
 R01H10A6A00T-00
 R01H10A7A00T-00
 R01H10A8A00T-00


Pos.	Part number	Description	Valid from → until	Qty
1	SP1637880	Power Module LiNX 120A Storm4 6&10km/h, Drive only		1
2	SP1637881	Power Module LiNX 120A Storm4 6km/h, Actuators & Light		1
3	SP1637882	Power Module LiNX 120A Storm4 10km/h, Actuators & Light		1

Bracket, Power Module

1639180.00

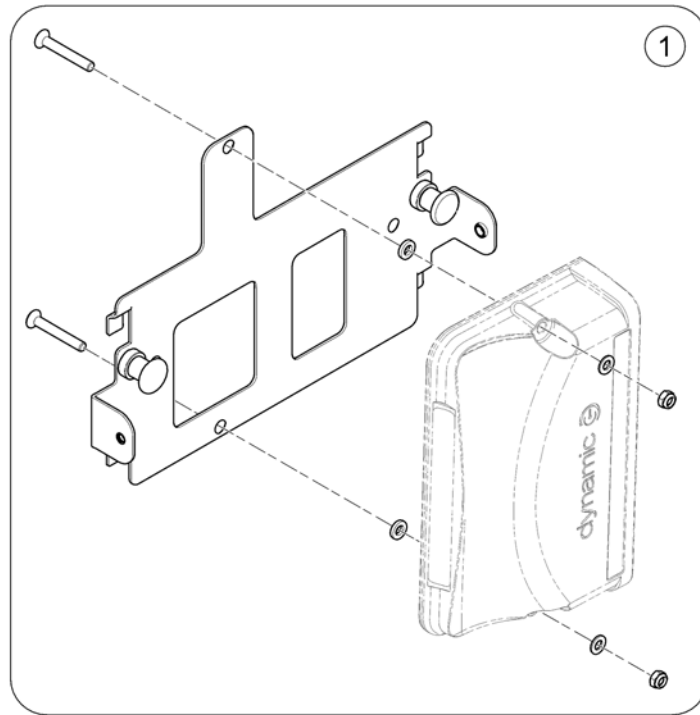


Pos.	Part number	Description	Valid from → until	Qty
1	SP1639173	Bracket, Power Module LiNX		1

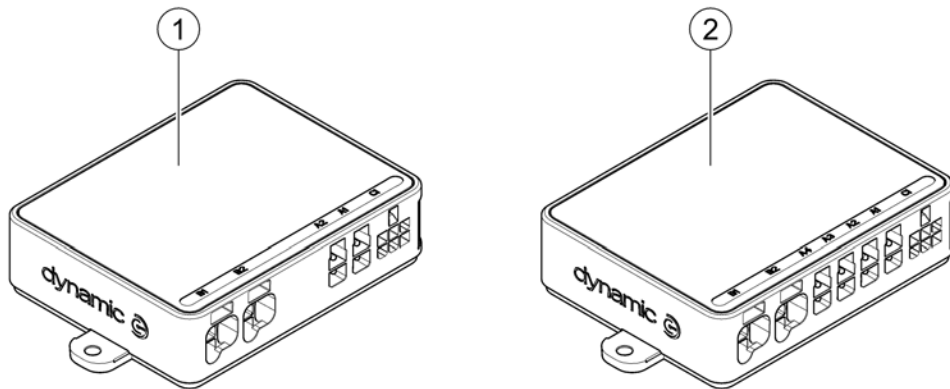

Pos.	Part number	Description	Valid from → until	Qty
1	SP1602883	LiNX 2-Channel Actuator Module (DLX-ACT200)		1
2	SP1602884	LiNX 4-Channel Actuator Module (DLX-ACT400)		1

G-Trac Sensor / Gyro

1639182.00

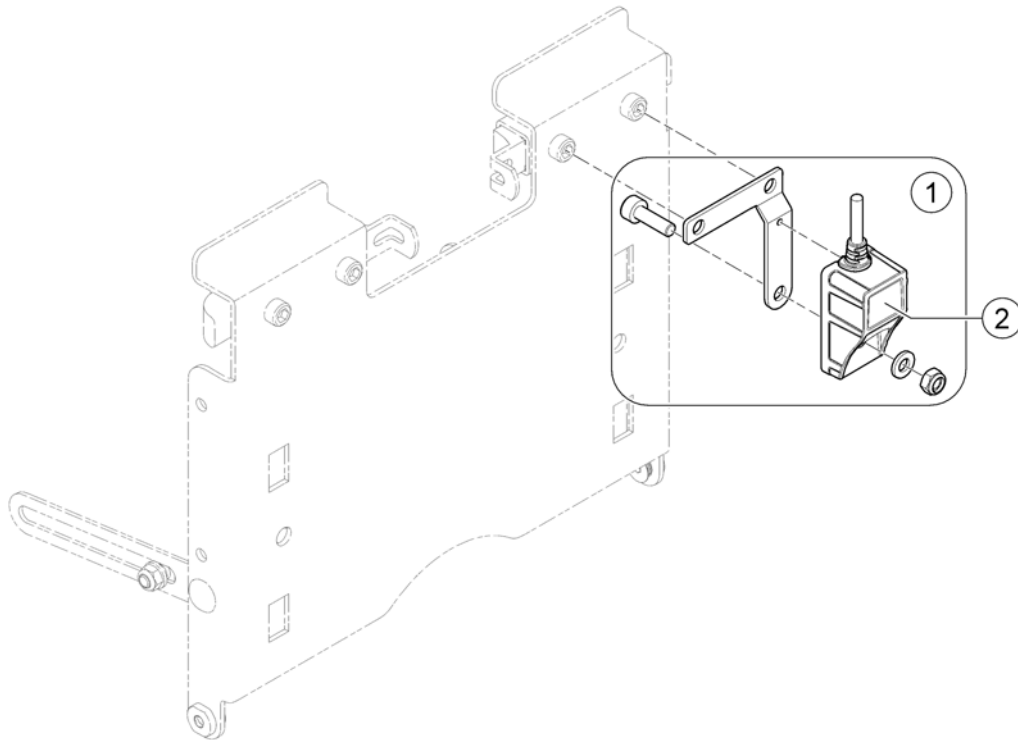

Pos.	Part number	Description	Valid from → until	Qty
1	SP1607817	G-Trac Sensor (Gyro) LiNX, cpl. with bracket		1
2	SP1602885	G-Trac Sensor (Gyro) LiNX DLX-GYR100		1


Pos.	Part number	Description	Valid from → until	Qty
-		User Manuals LiNX		1
-		Remotes & Brackets		1
-		Dual Control & Bracket		1

User Manuals LiNX

languages.iso

BG	Български	FR	Français	NL	Nederlands	SL	Slovenščina
CS	Čeština	HE	עברית	NO	Norsk	SV	Svenska
DA	Dansk	HR	Hrvatski	PL	Polski	UK	Українська
DE	Deutsch	HU	Magyar	PT	Português		
EN	English	IT	Italiano	RU	Русский		
ES	Español	JA	日本語	RO	Română		
ET	Eesti	LV	Latviešu	SA	العربية		
FI	Suomi	LT	Lietuvių	SK	Slovenčina		


Pos.	Part number	Description	Valid from → until	Qty
BG	SP1603635	User Manual LiNX - Bulgarian		1
CS	SP1603637	User Manual LiNX - Czech		1
DA	SP1603638	User Manual LiNX - Danish		1
DE	SP1603644	User Manual LiNX - German		1
EN	SP1602969	User Manual LiNX - English		1
ES	SP1603711	User Manual LiNX - Spanish		1
ET	SP1603641	User Manual LiNX - Estonian		1
FI	SP1603642	User Manual LiNX - Finnish		1
FR	SP1603643	User Manual LiNX - French		1
HR	SP1603636	User Manual LiNX - Croatian		1
HU	SP1603700	User Manual LiNX - Hungarian		1
IT	SP1603701	User Manual LiNX - Italian		1
LV	SP1603702	User Manual LiNX - Latvian		1
LT	SP1603703	User Manual LiNX - Lithuanian		1
NL	SP1603639	User Manual LiNX - Dutch		1
NO	SP1603704	User Manual LiNX - Norwegian		1
PL	SP1603705	User Manual LiNX - Polish		1
PT	SP1603706	User Manual LiNX - Portuguese		1
RU	SP1603707	User Manual LiNX - Russian		1
SK	SP1603799	User Manual LiNX - Slovak		1
SL	SP1603708	User Manual LiNX - Slovenian		1
SV	SP1603712	User Manual LiNX - Swedish		1

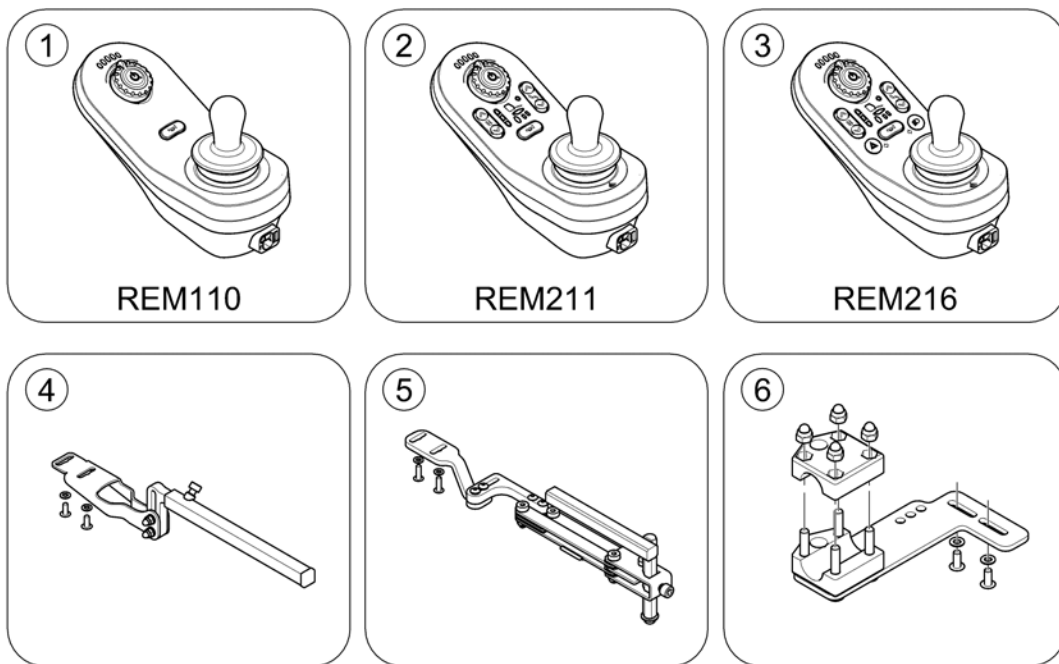

Pos.	Part number	Description	Valid from → until	Qty
1	-	Remote REM110		1
2	-	Remote REM211		1
3	-	Remote REM216		1
4	-	Remote Bracket, Standard		1
5	-	Remote Bracket, Swing away		1
6	-	Remote Bracket, Attendant		1

Remote REM110


1605135 iso

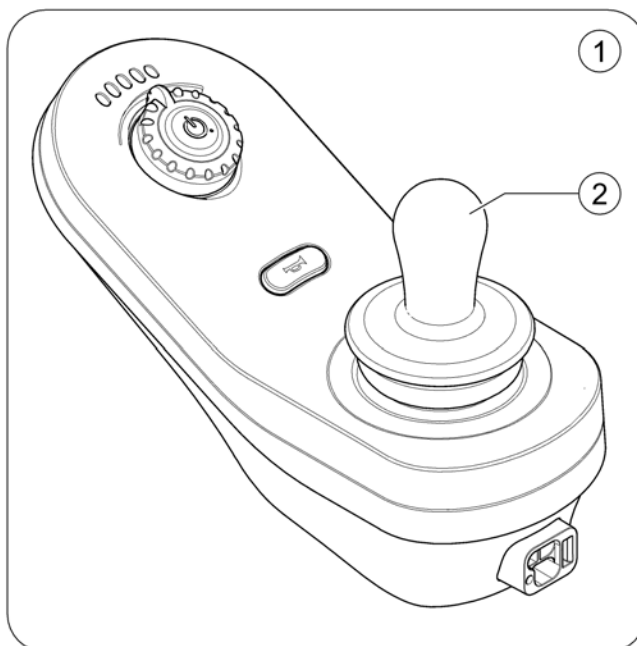

Pos.	Part number	Description	Valid from → until	Qty
1	SP1602806	LiNX Remote DLX-REM110 - Drive Only		1
2	SP1607776	Joystick Knob		1

Remote REM211


1605136 iso

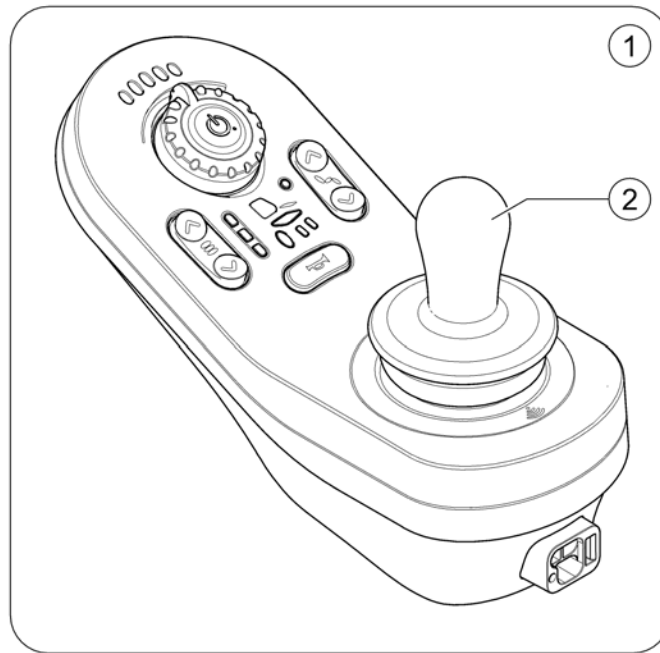

Pos.	Part number	Description	Valid from → until	Qty
1	SP1602825	LiNX Remote DLX-REM211 - Drive and Actuators		1
2	SP1607776	Joystick Knob		1

Remote REM216


1605137.iss

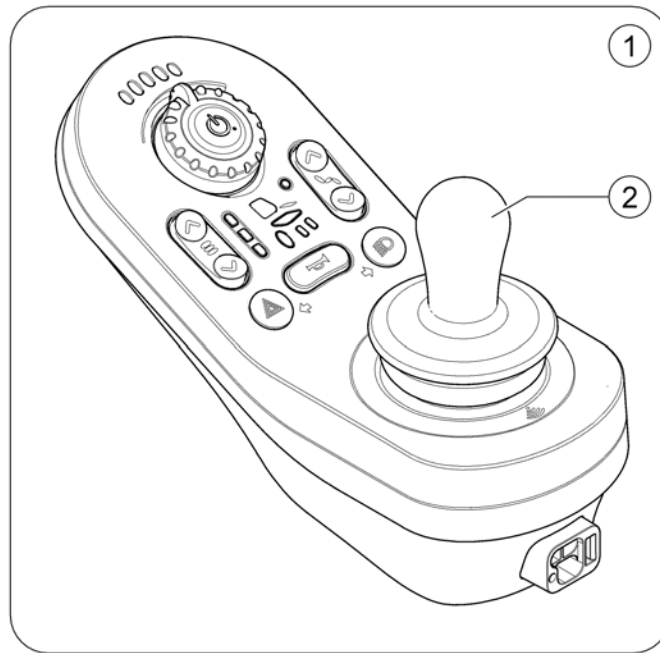

Pos.	Part number	Description	Valid from → until	Qty
1	SP1602826	LiNX Remote DLX-REM216 - Drive, Actuators and Lights		1
2	SP1607776	Joystick Knob		1

Remote Bracket, Standard


1605138 iso

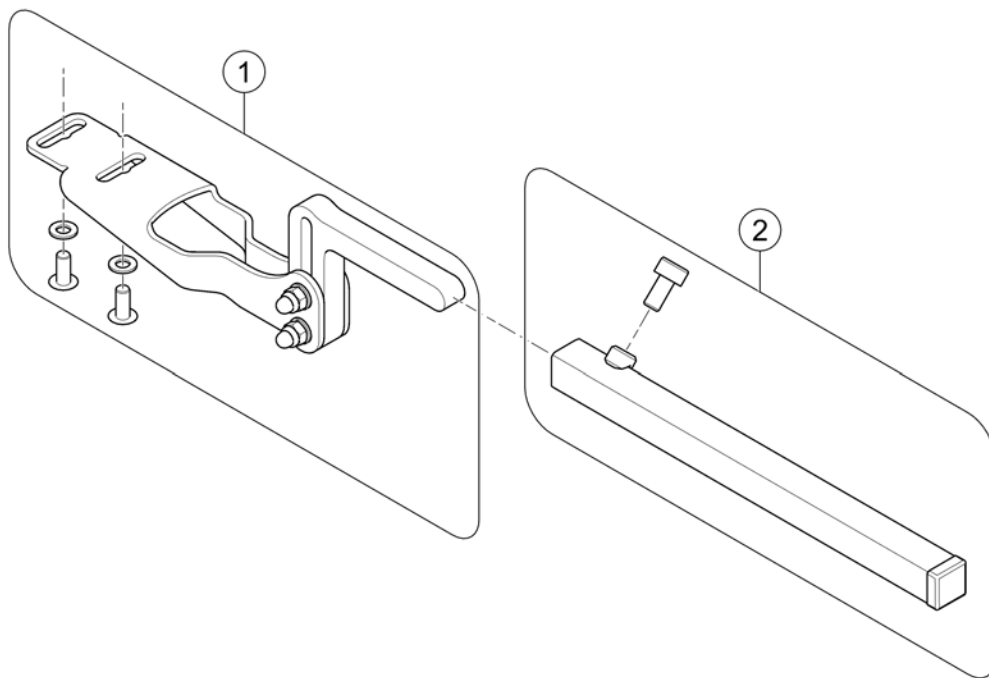

Pos.	Part number	Description	Valid from → until	Qty
1	SP1603417	LiNX Remote bracket, Standard		1
2	SP1603419	Remote support tube		1

Remote Bracket, Swing away


1605139 iso

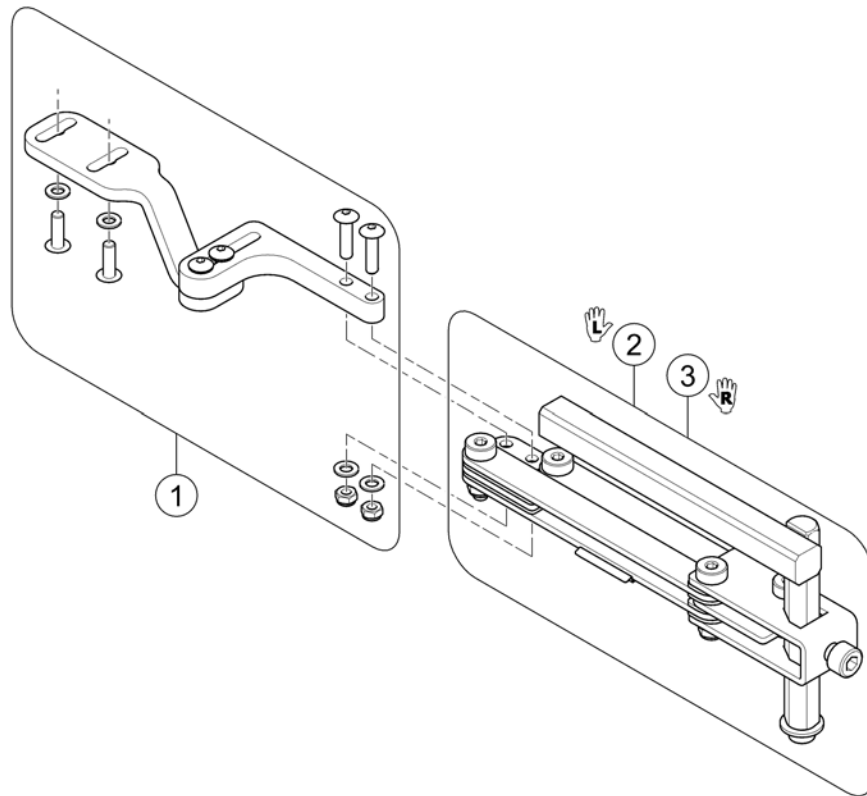

Pos.	Part number	Description	Valid from → until	Qty
1	SP1603418	LiNX Remote support for height adjustable & swing-away bracket		1
2	SP1496249	Remote bracket, height adjustable & swing away, LH		1
3	SP1496248	Remote bracket, height adjustable & swing away, RH		1

Remote Bracket, Attendant



1605185 iso

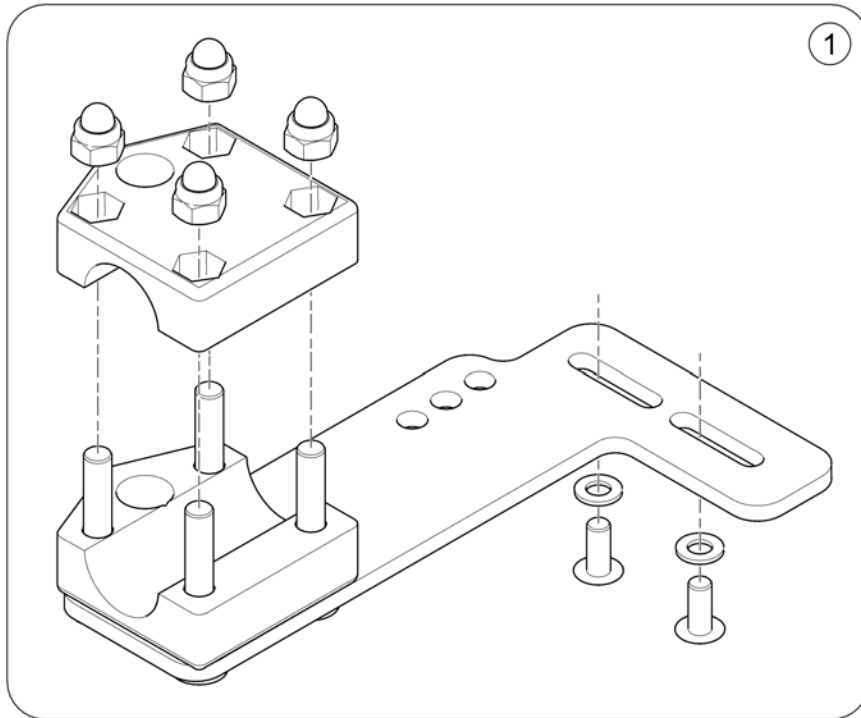


Pos.	Part number	Description	Valid from → until	Qty
1	SP1603363	LiNX Remote bracket, Attendant		1

Dual Control & Bracket

1605186.iss

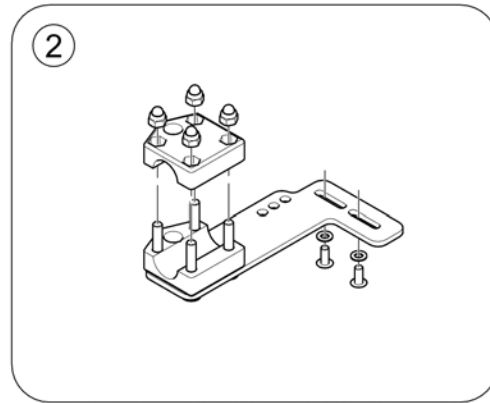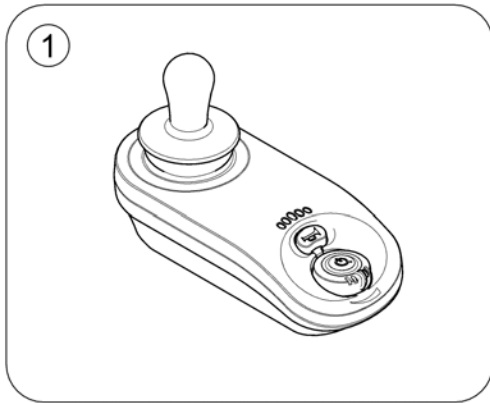

Pos.	Part number	Description	Valid from → until	Qty
1	-	Dual Control		1
2	-	Remote Bracket, Attendant		1

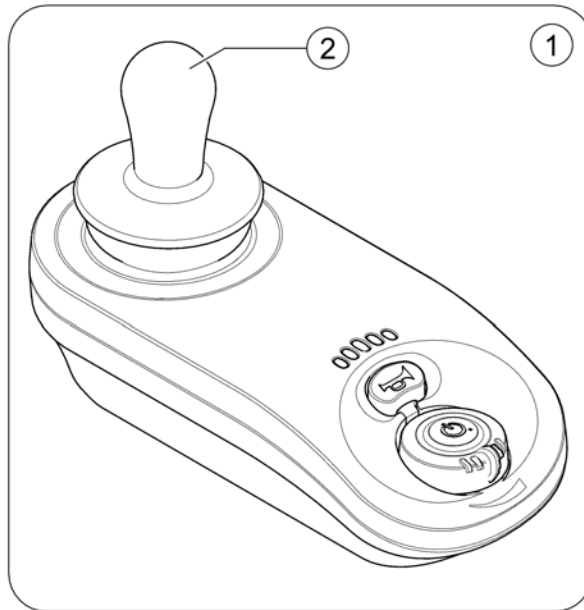

Pos.	Part number	Description	Valid from → until	Qty
1	SP1602792	LiNX Dual Control Remote DLX-REM050		1



Cables & Accessories

Pos.	Part number	Description	Valid from → until	Qty
-		Bus Cables		1
-		Actuator Cable		1
-		Lifter/Tilt-Module Cables		1
-		Light Cable		1
-		Light Board		1
-		4-way Expansion Block		1
-		Slow-Stop-Sensor - Modulite Seat		1
-		Slow-Stop-Sensor - Recaro Seat		1

Bus Cables

LiNX

1605413.jpg

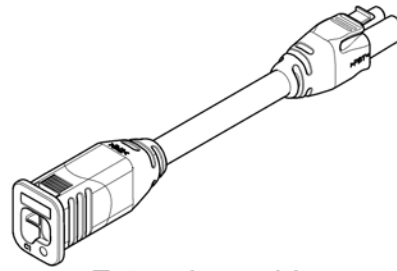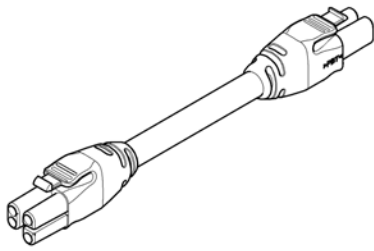

Bus cable

Lenght [mm]

- ① 300
- ② 500
- ③ 700
- ④ 1000
- ⑤ 1200
- ⑥ 1500
- ⑦ 1700
- ⑧ 2000
- ⑨ 2500

Extension cable


Lenght [mm]

- ⑩ 640
- ⑪ 900


Pos.	Part number	Description	Valid from → until	Qty
1	SP1602888	LiNX Bus cable 300mm (GLM-BUS030)		1
2	SP1602889	LiNX Bus cable 500mm (GLM-BUS050)		1
3	SP1602890	LiNX Bus cable 700mm (GLM-BUS070)		1
4	SP1602891	LiNX Bus cable 1000mm (GLM-BUS100)		1
5	SP1603489	LiNX Bus cable 1200mm (GLM-BUS120)		1
6	SP1602892	LiNX Bus cable 1500mm (GLM-BUS150)		1
7	SP1602893	LiNX Bus cable 1700mm (GLM-BUS170)		1
8	SP1602894	LiNX Bus cable 2000mm (GLM-BUS200)		1
9	SP1602904	LiNX Bus cable 2500mm (GLM-BUS250)		1
10	SP1602905	LiNX Extension cable 640mm (GLM-EXT064)		1
11	SP1602906	LiNX Extension cable 900mm (GLM-EXT090)		1

Actuator Cable

1505463 iso

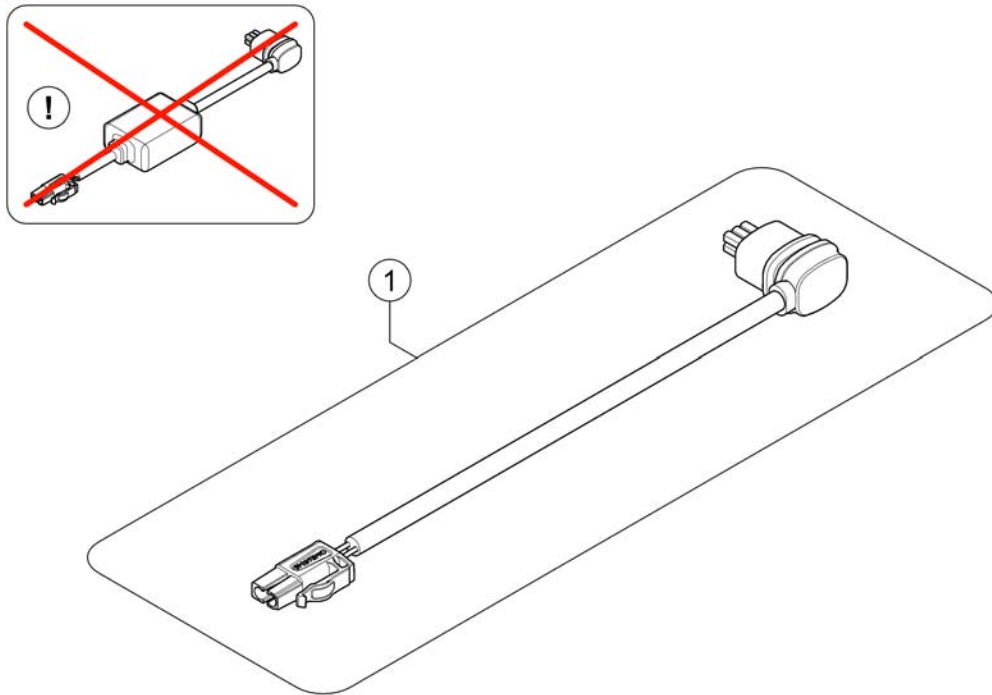


Pos.	Part number	Description	Valid from → until	Qty
!	-	No longer available - please refer to Replacement Info for which part is to be used instead.		0
1	SP1607105	LiNX Actuator cable LA23 - 950mm <i>Only Backrest Recliner - Replaces SP1605680</i>		1
1	SP1607124	LiNX Actuator cable LA23 - 1500mm <i>Replaces SP1605815</i>		1

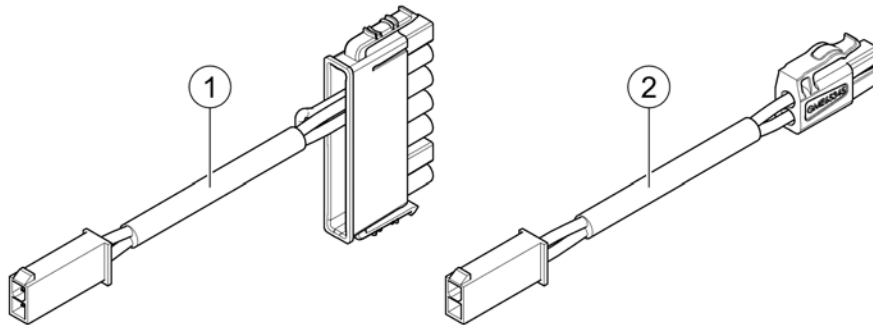


Pos.	Part number	Description	Valid from → until	Qty
1	SP1607899	Sensor Cable LiNX - 300mm		1
2	SP1573204	Adater cable - 500mm		1
2	SP1576062	Extension Cable - 1100mm		1

Light Cable

160623 iso

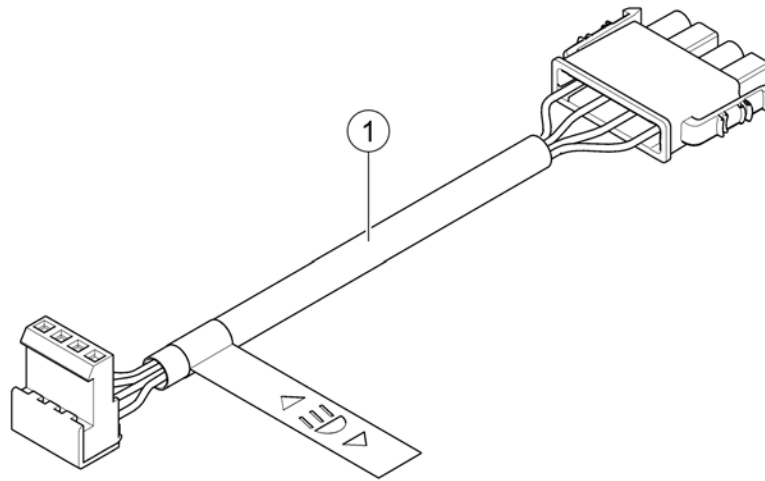


Pos.	Part number	Description	Valid from → until	Qty
1	SP1603336	LiNX Light cable		1

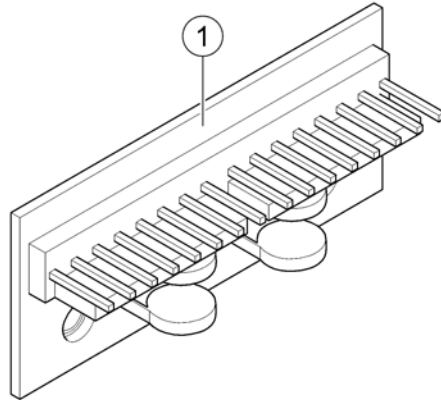

Pos.	Part number	Description	Valid from → until	Qty
1	1415004	Central Light Distributor Circuit Board ACS/Shark		1

4-way Expansion Block


1639196.00

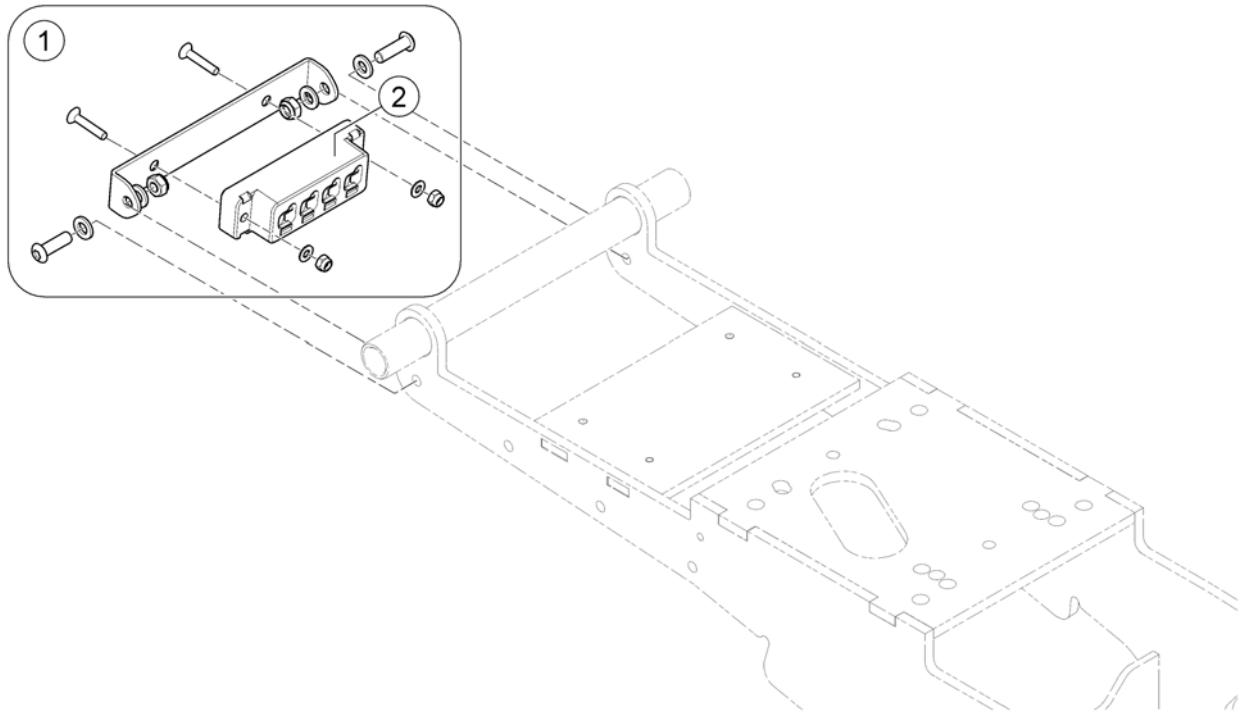

Pos.	Part number	Description	Valid from → until	Qty
1	SP1607842	4-way Expansion Block LiNX, cpl. with mounting hardware		1
2	SP1602907	4-way Expansion Block, LiNX, GLM-CONX4 (single part)		1

Slow-Stop-Sensor - Modulite Seat


1609178/00

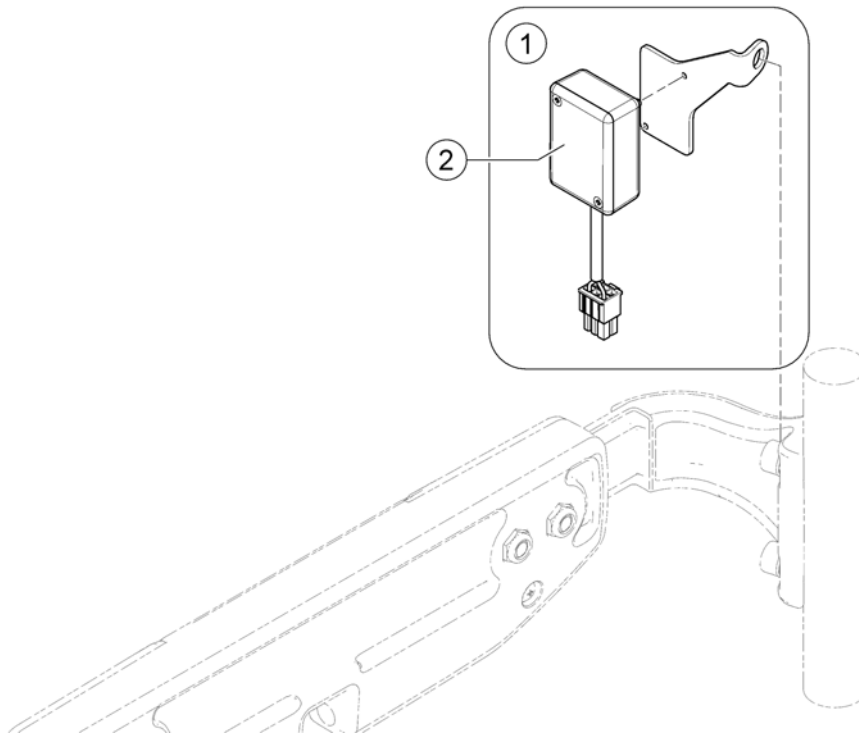

Pos.	Part number	Description	Valid from → until	Qty
1	SP1607887	Slow-Stop-Sensor LiNX, cpl. with bracket for Modulite Seat		1
2	SP1604482	Slow-Stop-Sensor LiNX		1

Slow-Stop-Sensor - Recaro Seat





1639179.00

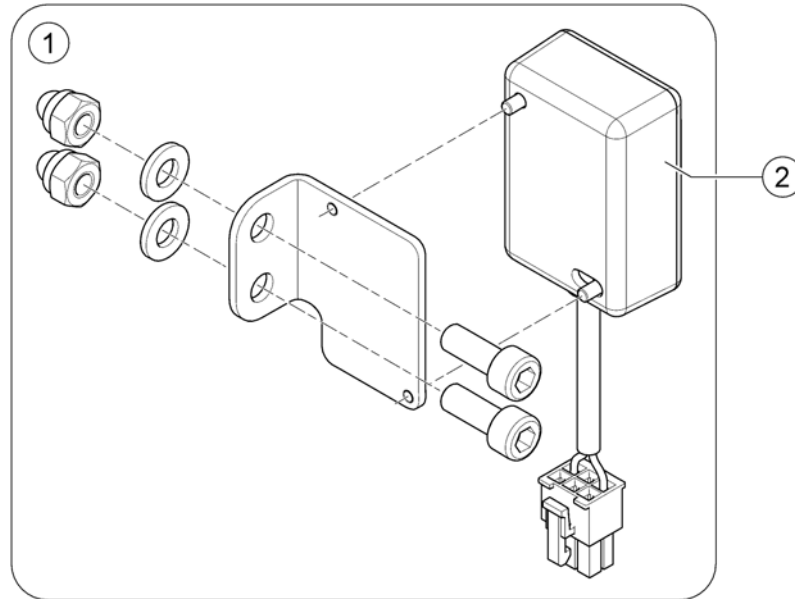

Pos.	Part number	Description	Valid from → until	Qty
1	SP1607888	Slow-Stop-Sensor LiNX, cpl. with mounting hardware for Recaro Seat		1
2	SP1603338	LiNX Slow-Stop-Sensor, cpl.		1

Components



1608154-00

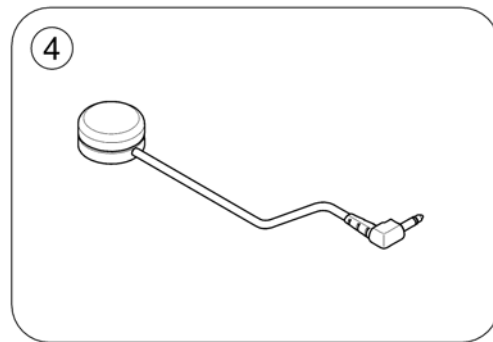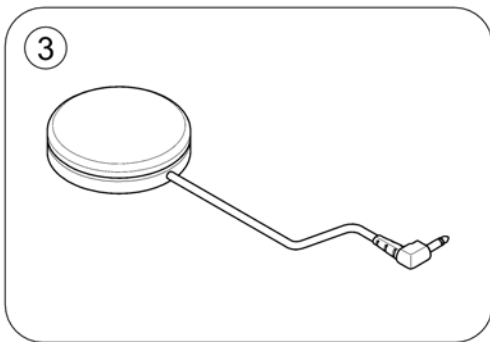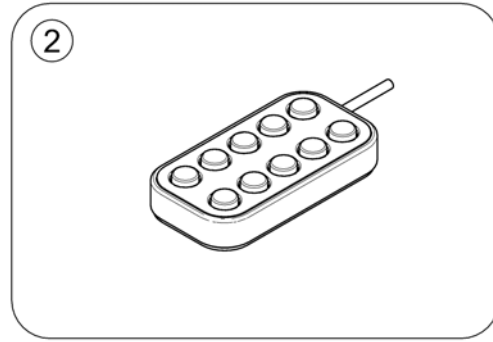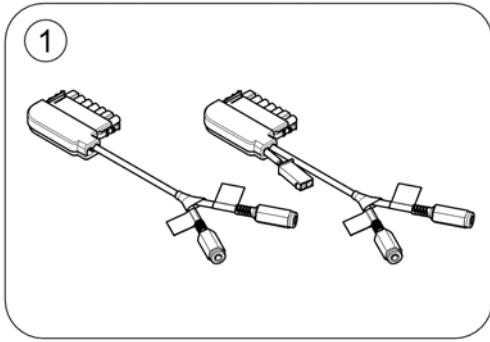

Pos.	Part number	Description	Valid from → until	Qty
1	-	Accessory cables		1
2	-	10-way switch & brackets		1
3	-	Buddy Button		1
4	-	Piko Button		1

Accessory cables





1608109-00

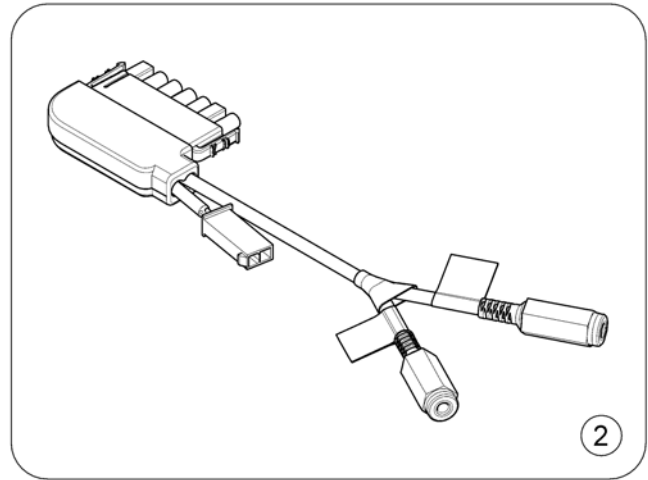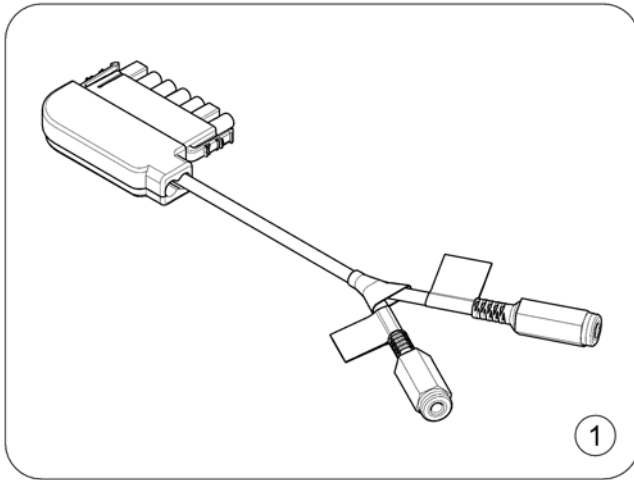

Pos.	Part number	Description	Valid from → until	Qty
1	SP1637339	Accessory cable LiNX (w/o lifter)		1
2	SP1608269	Accessory cable LiNX (with lifter)		1

10-way switch & brackets


1608155.kim

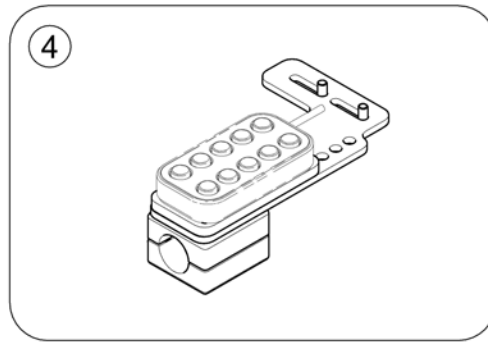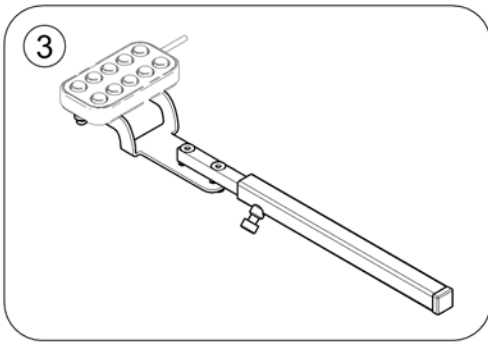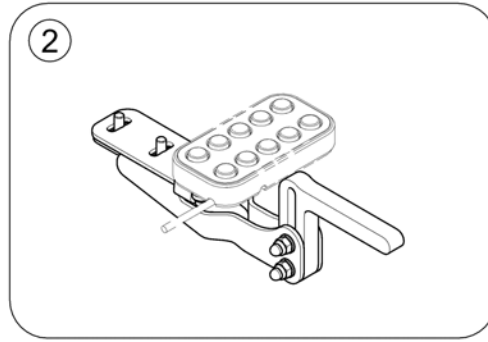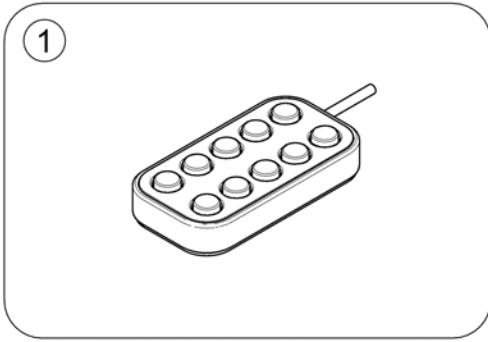

Pos.	Part number	Description	Valid from → until	Qty
1	-	10-way switch		1
2	-	Bracket, front		1
3	-	Bracket, opposite side		1
4	-	Bracket, rear		1

10-way switch


1608110.iso

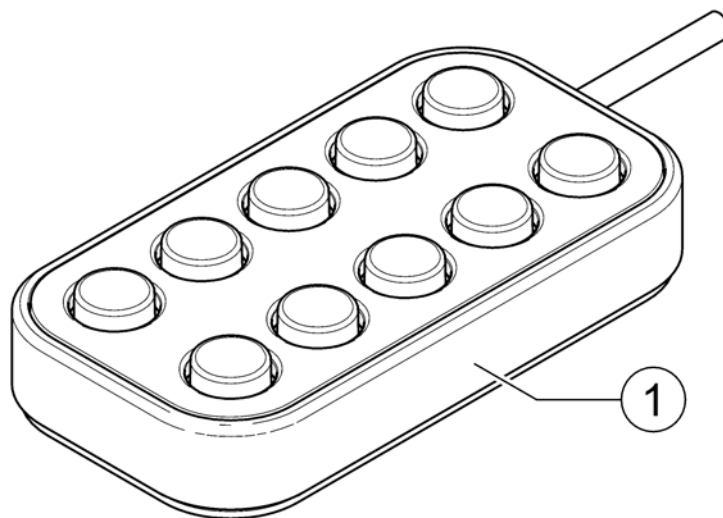

Pos.	Part number	Description	Valid from → until	Qty
1	SP1602838	LiNX 10-way switch (DLX-SW100)		1

Bracket, front


1008157.00

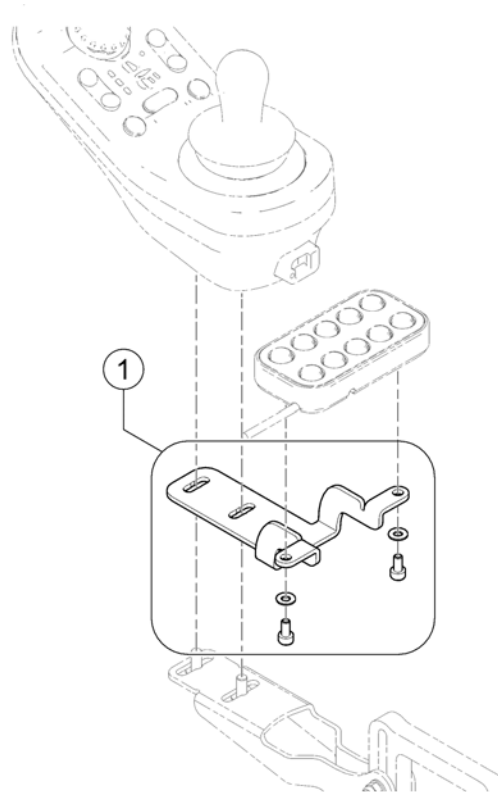

Pos.	Part number	Description	Valid from → until	Qty
1	SP1606388	Bracket 10-way switch - front		1

Bracket, opposite side


1639748-00

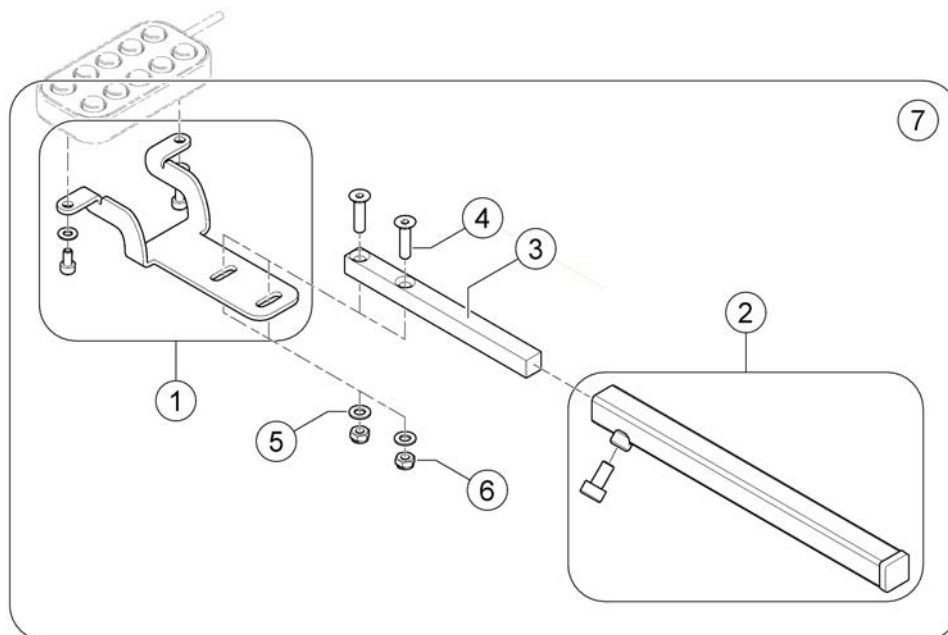

Pos.	Part number	Description	Valid from → until	Qty
1	SP1606388	Bracket 10-way switch - front		1
2	SP1603419	Remote support tube		1
3	1440437-9005	Joystick support Standard		1
4	1417938	Countersunk bolt DIN7991-M05x20-8.8 zc		2
5	1422890	Washer DIN125-Ø5,3 bk		2
6	F20111	Hexagon nut DIN985-M05 bk		2
7	SP1606371	Bracket, opposite side, cpl.		1

Bracket, rear




1608165.00

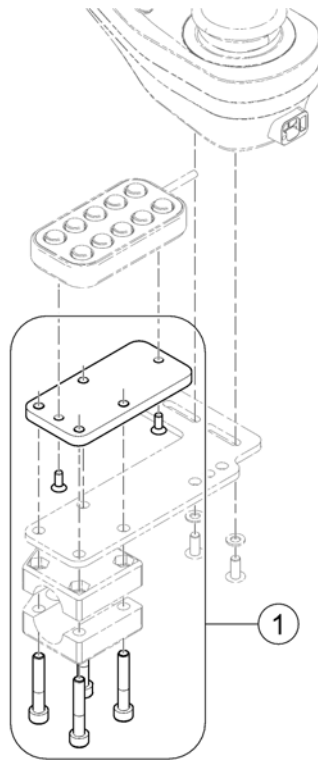

Pos.	Part number	Description	Valid from → until	Qty
1	SP1606389	Bracket 10-way switch - rear		1

Storm4 TTPlus

FDX79.00

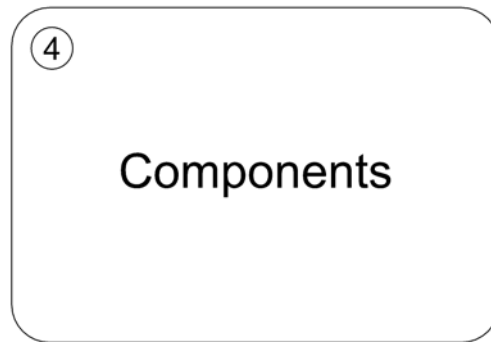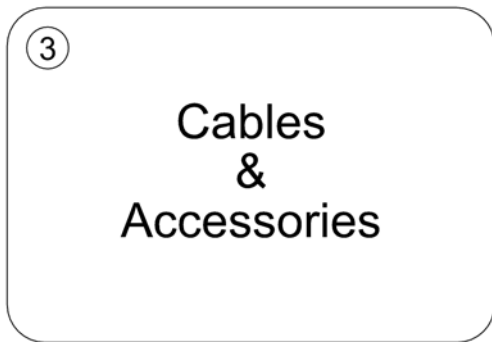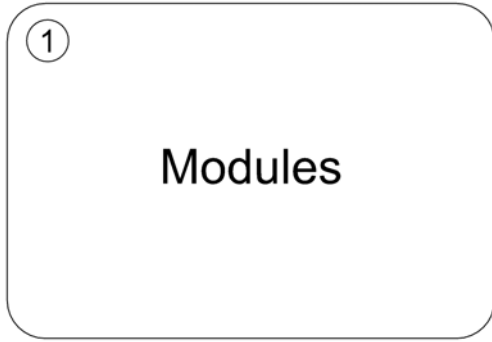


Pos.	Part number	Description	Valid from → until	Qty
1	-	ACS Modules		0
2	-	ACS Remotes & Brackets		0
3	-	ACS Cables & Accessories		0
4	-	ACS Components		0

ACS Modules

Pos.	Part number	Description	Valid from → until	Qty
-		Powermodule GGB & Bracket		0
-		CLAM & Bracket		0
-		Relais & bracket		0
-		Lightboard GGB		0


Powermodule GGB & Bracket

1581910 iso

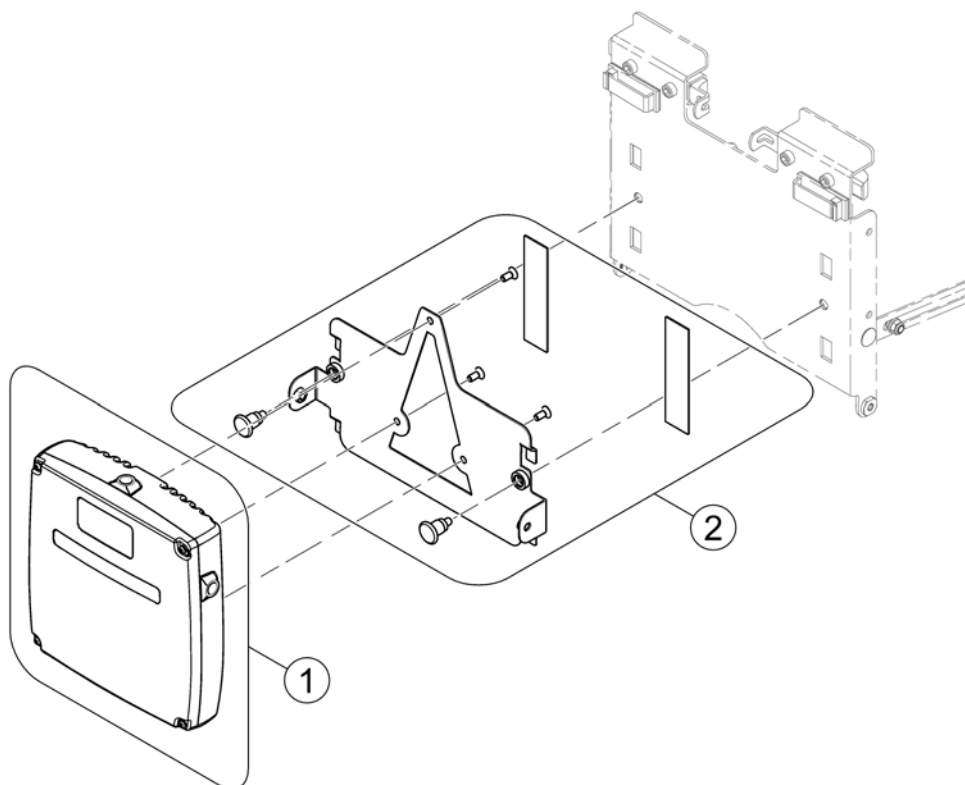

Pos.	Part number	Description	Valid from → until	Qty
1	SP1568490	Powermodule DX-GB2		1
2	SP1582003	Bracket Powermodule GGB2		1


1581911.sp

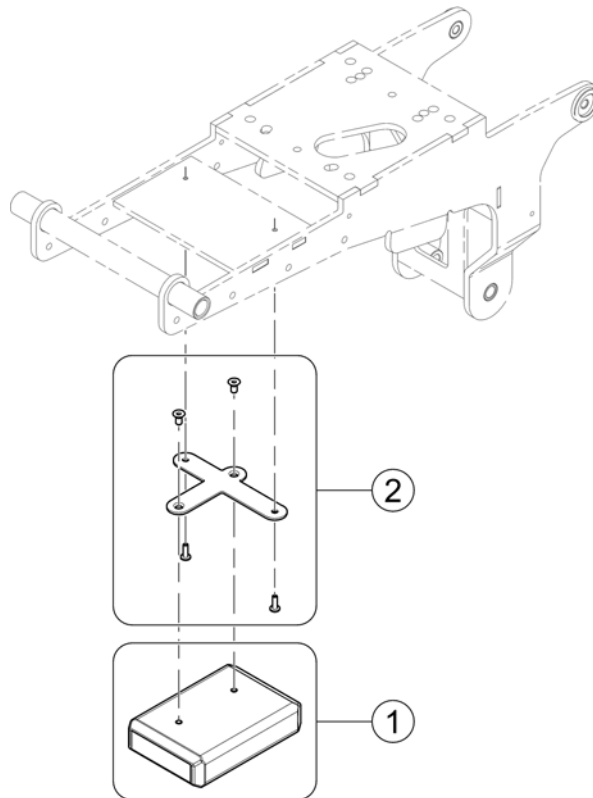



Pos.	Part number	Description	Valid from → until	Qty
1	P10081	ACS Light/Actuator Module (Clam)		1
2	SP1582004	Bracket CLAM		1

Relais & bracket

1603457.kim

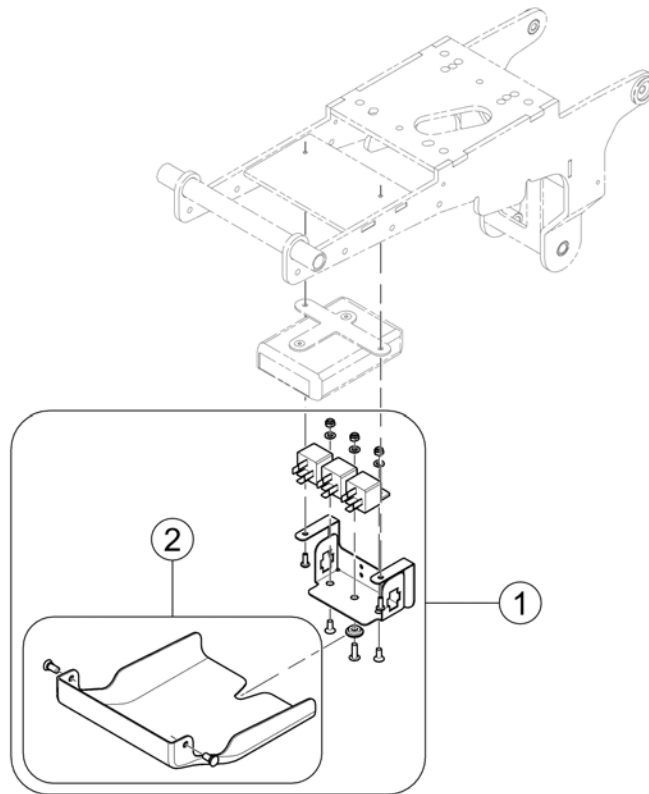


Pos.	Part number	Description	Valid from → until	Qty
1	SP1580133	Relais, bracket & cover, cpl.		1
2	SP1580197	Relais cover, cpl.		1

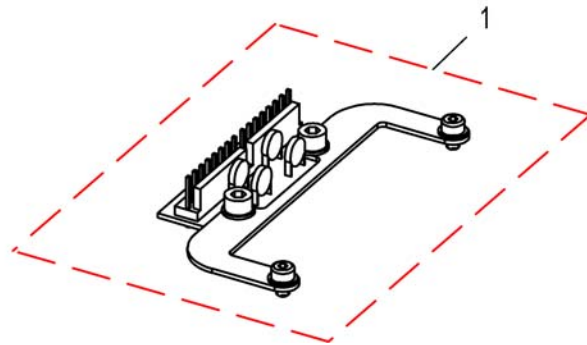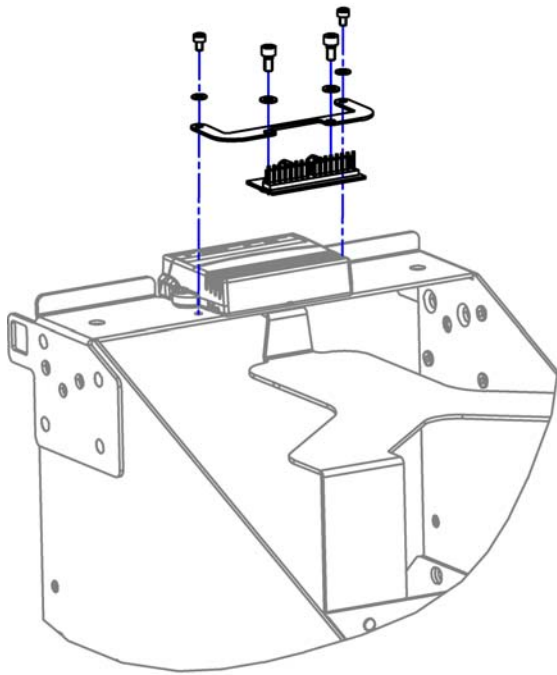


Pos.	Part number	Description	Valid from → until	Qty
1	SP1587248	Light Board, cpl. <i>LED only! [1437]</i>		1
1	SP1528890	Light board, cpl. <i>German homologation only! [1367]</i>		1

ACS Remotes & Brackets

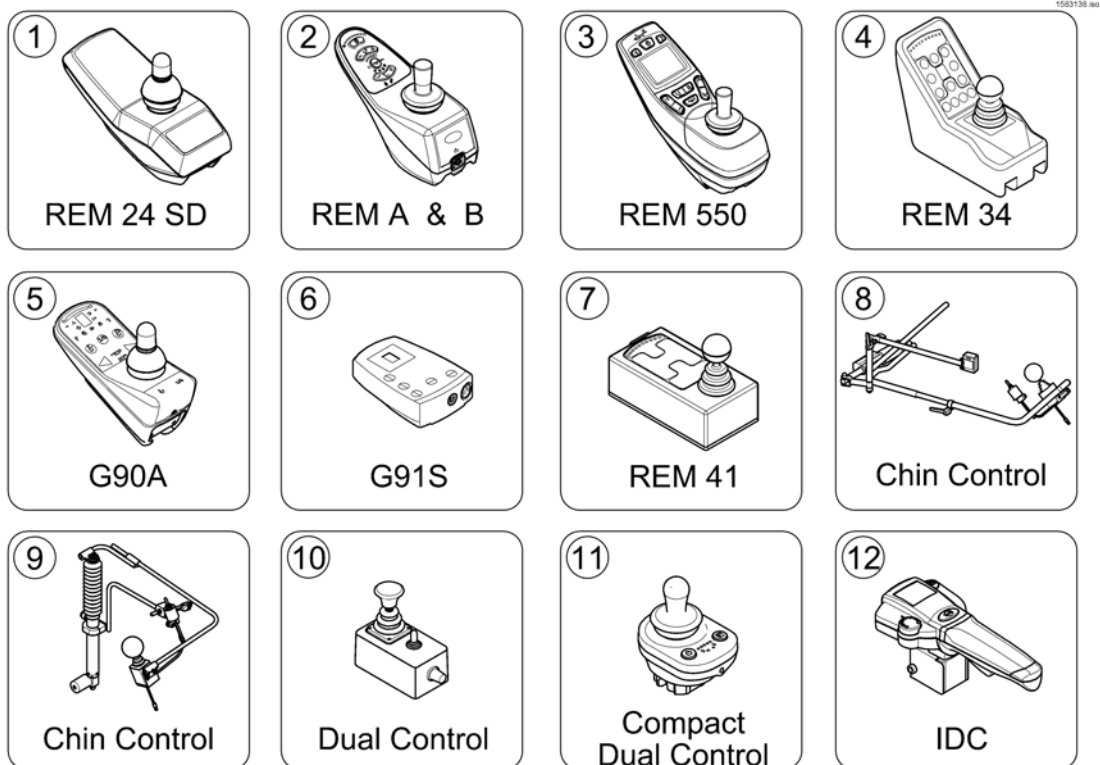

Pos.	Part number	Description	Valid from → until	Qty
1	-	REM 24 SD with brackets <i>[1601/1602]</i>		1
2	-	REM A & B with brackets <i>[1638/1639/1643/1644]</i>		1
3	-	REM 550 with brackets <i>[1617/1618]</i>		1
4	-	REM 34 with brackets <i>[1611]</i>		1
5	-	G90A with brackets <i>[1614/1616]</i>		1
6	-	G91S with brackets <i>[1615]</i>		1
7	-	Centre control REM 41 <i>[1607/1608]</i>		1
8	-	Chin Control, small (=> 16/12/31) <i>[1609]</i>		1
9	-	Chin Control - Modulite Seat (15/10/01 =>) <i>[1719]</i>		1
10	-	Dual Control with brackets <i>[1291/1292]</i>	→ 2015-06-30	1
11	-	Compact Dual Control with brackets <i>[1720]</i>	2015-07-01 →	1
12	-	Intuitive Dual Control (IDC) with bracket <i>[1645]</i>		1

REM 24 SD with brackets

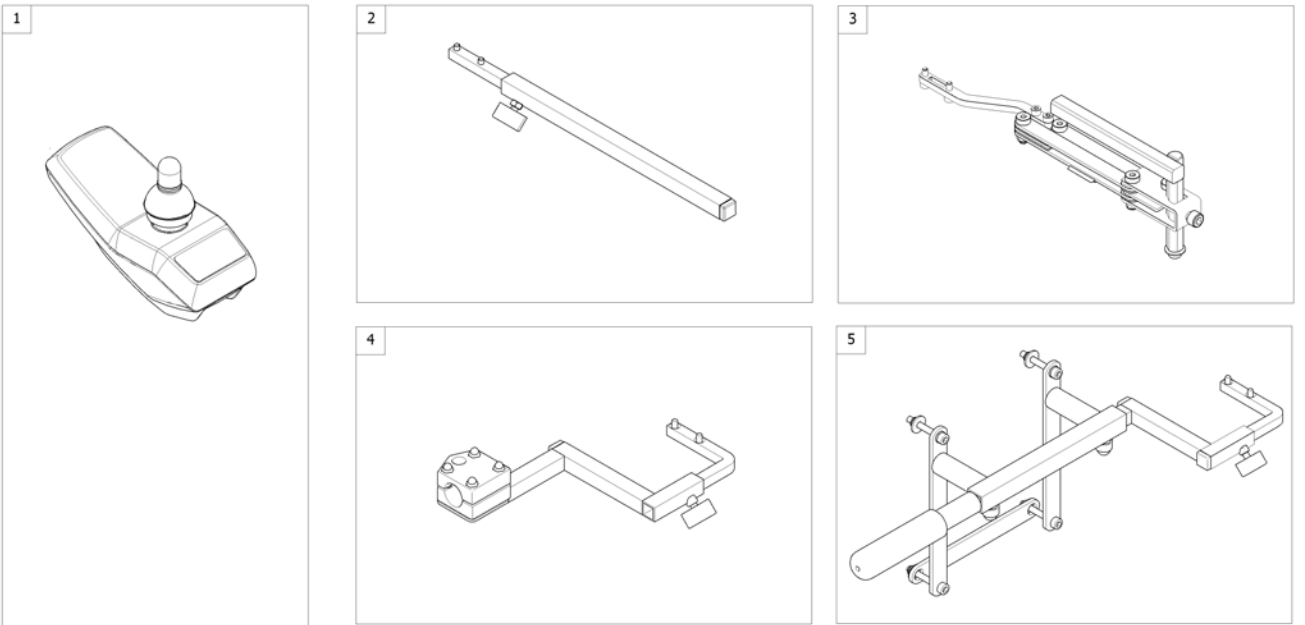


1563648 no


Pos.	Part number	Description	Valid from → until	Qty
1	-	Remote REM 24SD <i>[1601/1602]</i>		0
2	-	Remote Bracket Standard <i>[1601+1207/1208]</i>		1
3	-	Remote Bracket Swing away & height adjust. <i>[1601+1707/1708]</i>		1
4	-	Remote Bracket Attendant for Standard-/EasyAdapt-Seat <i>[1602+1207/1208]</i>		1
5	-	Remote Bracket Attendant for Recaro-Seat <i>[1602+1207/1208]</i>		1

REM 550 with brackets

1563647 no

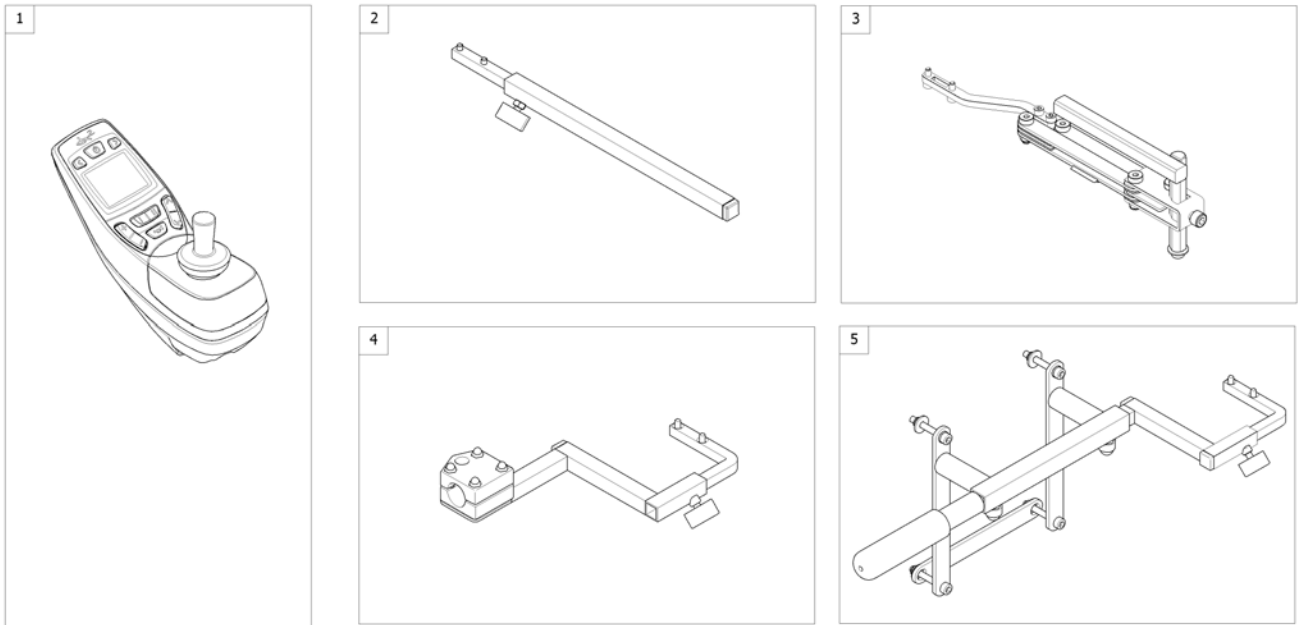


Pos.	Part number	Description	Valid from → until	Qty
1	SP1523444	Remote REM550 <i>[1617/1618]</i>		1
2	-	Remote Bracket Standard <i>[1617+1207/1208]</i>		1
3	-	Remote Bracket Swing away & height adjust. <i>[1617+1707/1708]</i>		1
4	-	Remote Bracket Attendant for Standard-/EasyAdapt-Seat <i>[1618+1207/1208]</i>		1
5	-	Remote Bracket Attendant for Recaro-Seat <i>[1618+1207/1208]</i>		1

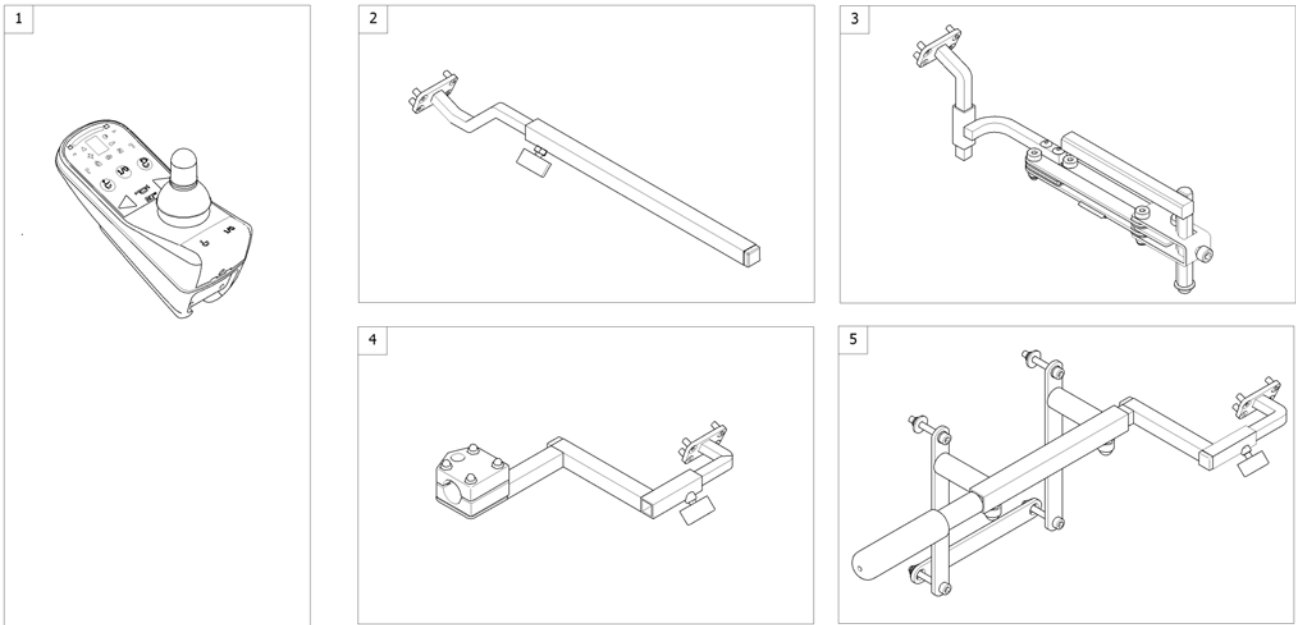

Pos.	Part number	Description	Valid from → until	Qty
1	SP1504720	Remote G90A [1614/1616]		1
2	-	Remote Bracket Standard [1614+1207/1208]		1
3	-	Remote Bracket Swing away & height adjust. [1614+1707/1708]		1
4	-	Remote Bracket Attendant for Standard-/EasyAdapt-Seat [1616+1207/1208]		1
5	-	Remote Bracket Attendant for Recaro-Seat [1616+1207/1208]		1

Remote Bracket Attendant for Standard-/EasyAdapt-Seat


1560853.iso

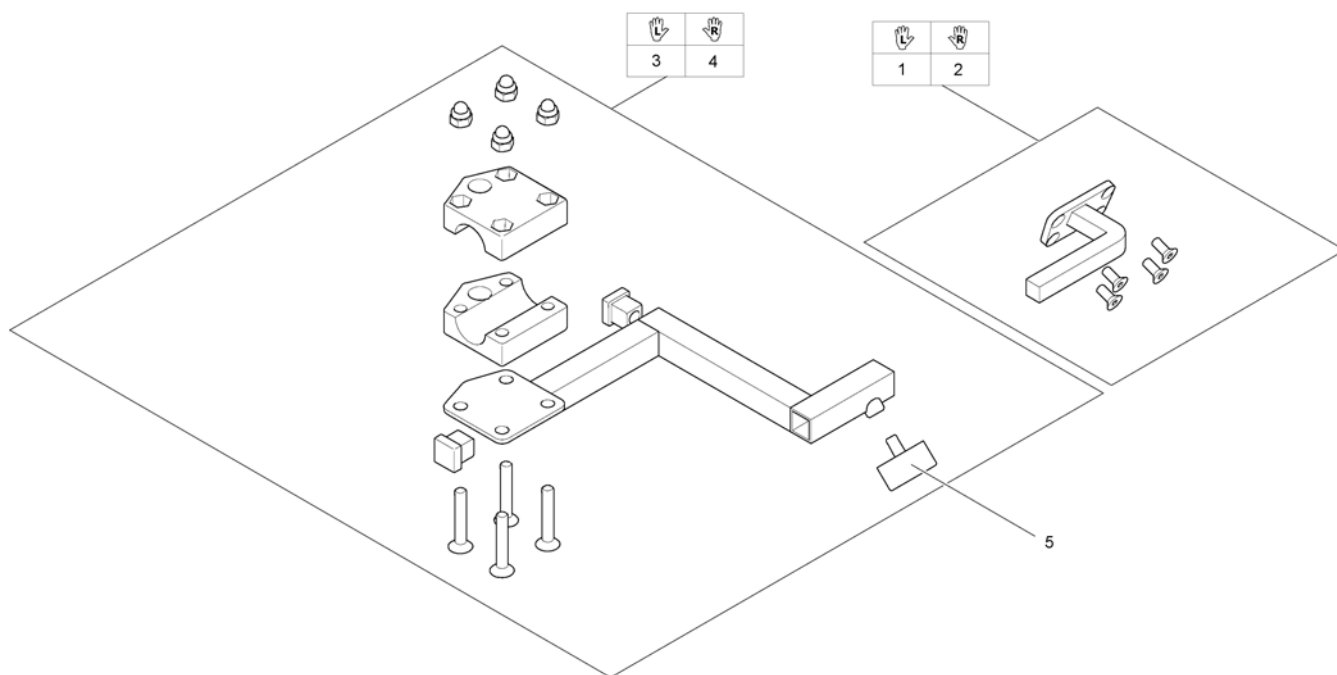

Pos.	Part number	Description	Valid from → until	Qty
1	SP1509435	Remote Support attendant control G90A, LH		1
2	SP1509439	Remote Support attendant control G90A, RH		1
3	SP1422937	Bracket for attendant control LH		1
4	SP1422936	Bracket for attendant control RH		1
5	L31001	Wing Screw M6X10		1

Compact Dual Control with brackets

1568802 Rev

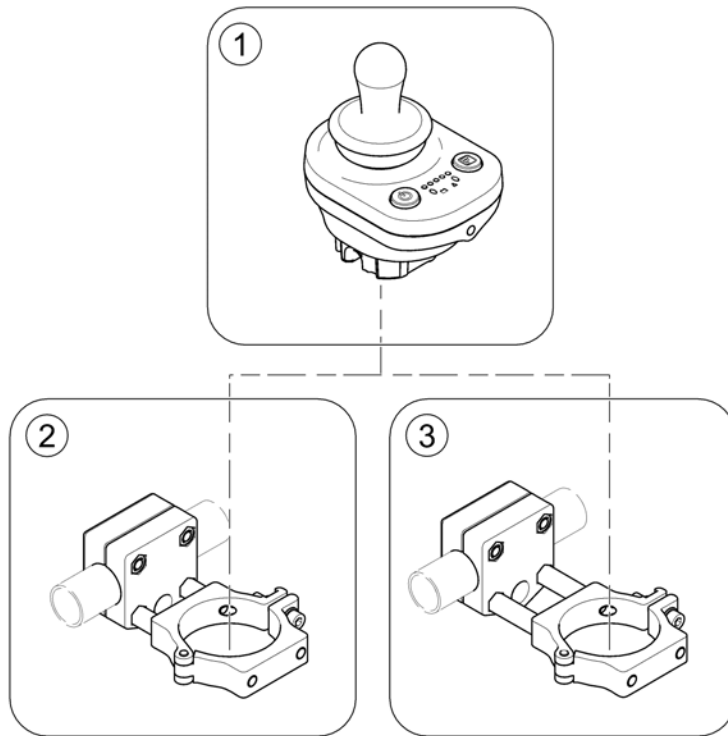

Pos.	Part number	Description	Valid from → until	Qty
1	-	Compact Dual Control		1
2	-	Bracket, Compact Dual Control - short		1
3	-	Bracket, Compact Dual Control - long		1

Intuitive Dual Control (IDC) with bracket


Pos.	Part number	Description	Valid from → until	Qty
-	-	IDC with bracket		1
-	-	Height adjustable bracket IDC		1

ACS Cables & Accessories

Pos.	Part number	Description	Valid from → until	Qty
-		Main cable harness		0
-		Plug Sheets el. Iegrest GGB		0
-		T-Adapter		0
-		T-Adapter Modulite Seat		0
-		Programming Kits		0
-		Magnetkey for Remotes		0

Main cable harness


1583074.iss

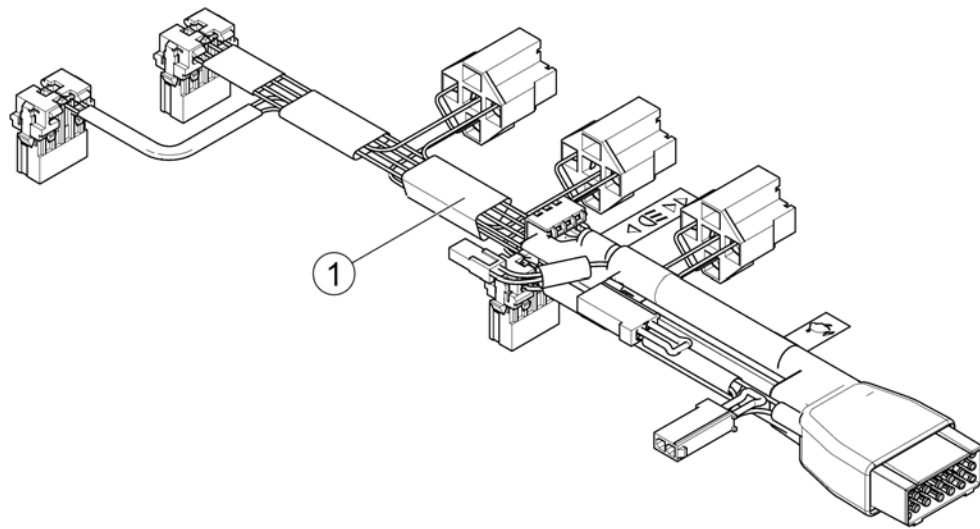

Pos.	Part number	Description	Valid from → until	Qty
1	SP1579550	Main Cable Harness Storm4 TT		1

Plug Sheets el. legrest GGB


156930 Ita

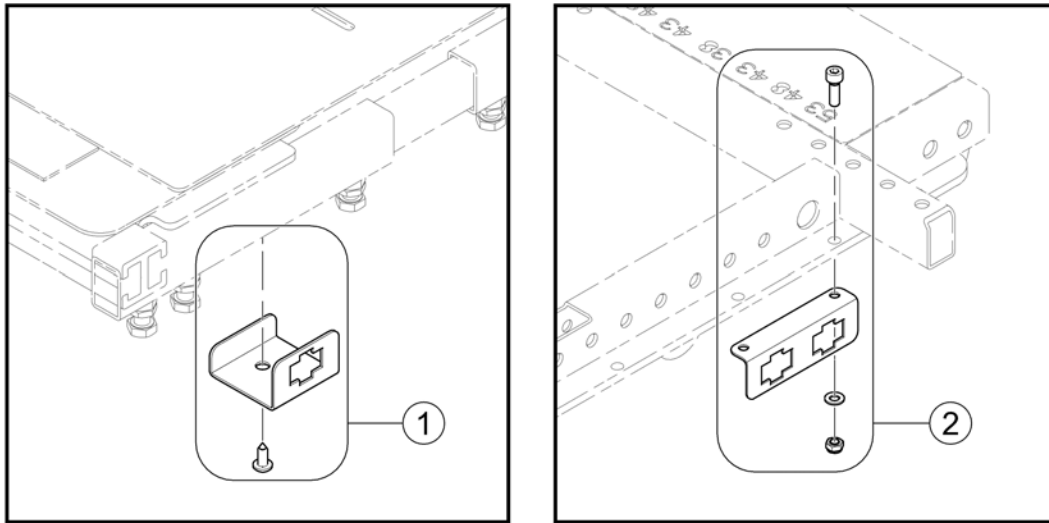


Pos.	Part number	Description	Valid from → until	Qty
1	SP1577086	Plug Sheet GGB - Modulite Seat		1
2	SP1581723	Plug Sheet GGB - EasyAdapt Seat		1

T-Adapter

1581912.iso

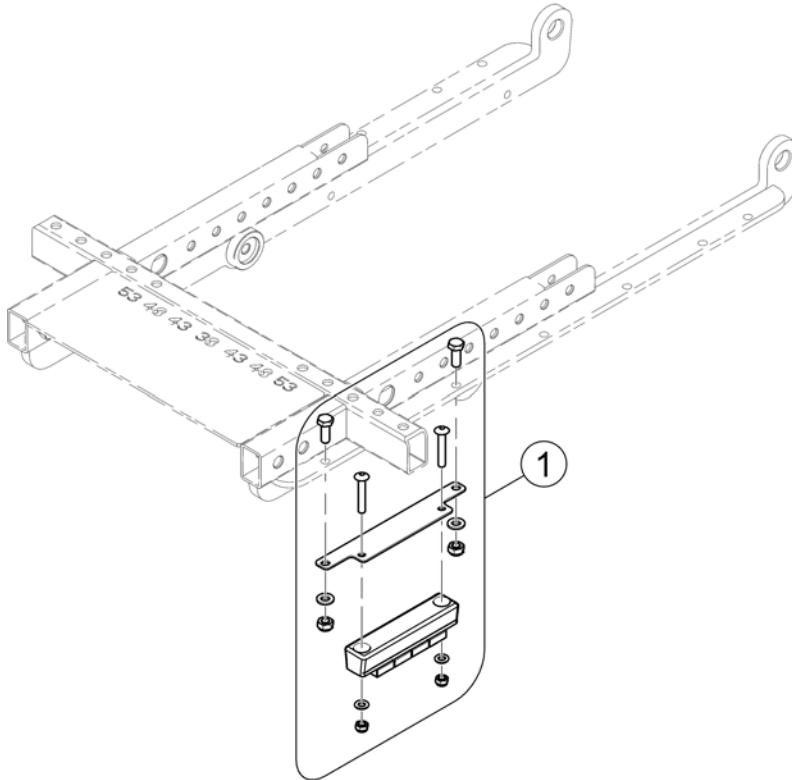

Pos.	Part number	Description	Valid from → until	Qty
1	SP1579548	T-Adapter GGB2, cpl.		1

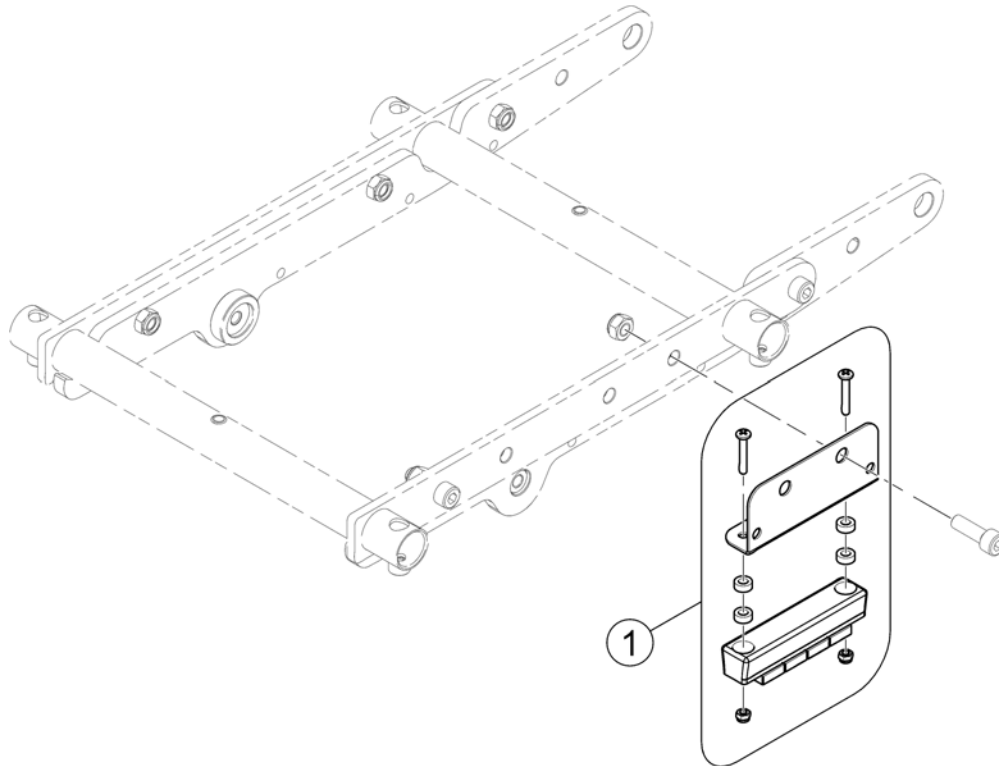


Pos.	Part number	Description	Valid from → until	Qty
1	SP1579547	T-Adapter Modulite GGB2, cpl.		1

Storm4

Pos.	Part number	Description	Valid from → until	Qty
-		Batteries & Accessories		0
-		Chargers		1

Batteries & Accessories


1581915.iso

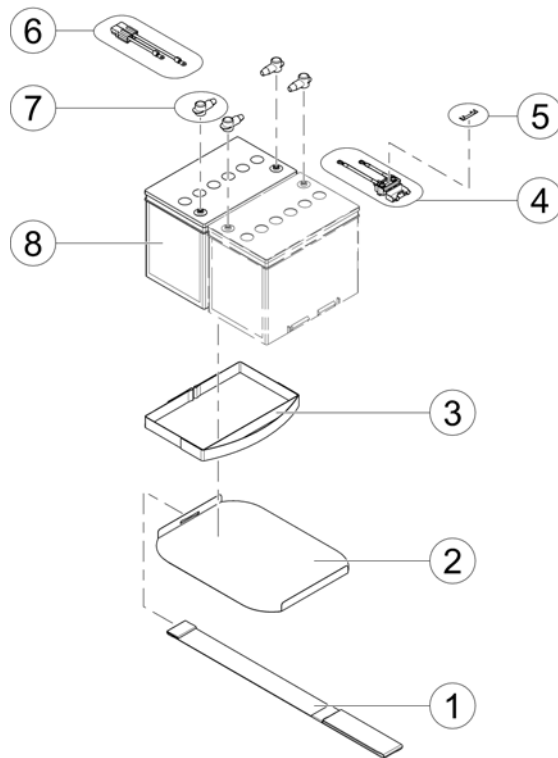

Pos.	Part number	Description	Valid from → until	Qty
-		Important advice! Replace batteries in pairs. Never mix: manufactures, date codes or technologies (Gel with AGM batteries)		1
1	SP1531651	Battery Strap		1
2	SP1524214	Battery carrier sheet		1
3	SP1518073	Battery strap		1
4	SP1525580	Fuse cable		1
-	SP1588075	Cable Tie Set V0 (10 of each T18R/T30R/T50R)		1
5	1433734	Strip Fuse 80A		1
6	SP1524666	Battery cable ACS		1
6	SP1607814	Battery Cable LiNX	2017-01-01 →	1
7	P52996	Battery terminal cover		4
8	P50016	Battery, 12V/70-73,6 Ah C20, Gel		2

Chargers

1589799.iso

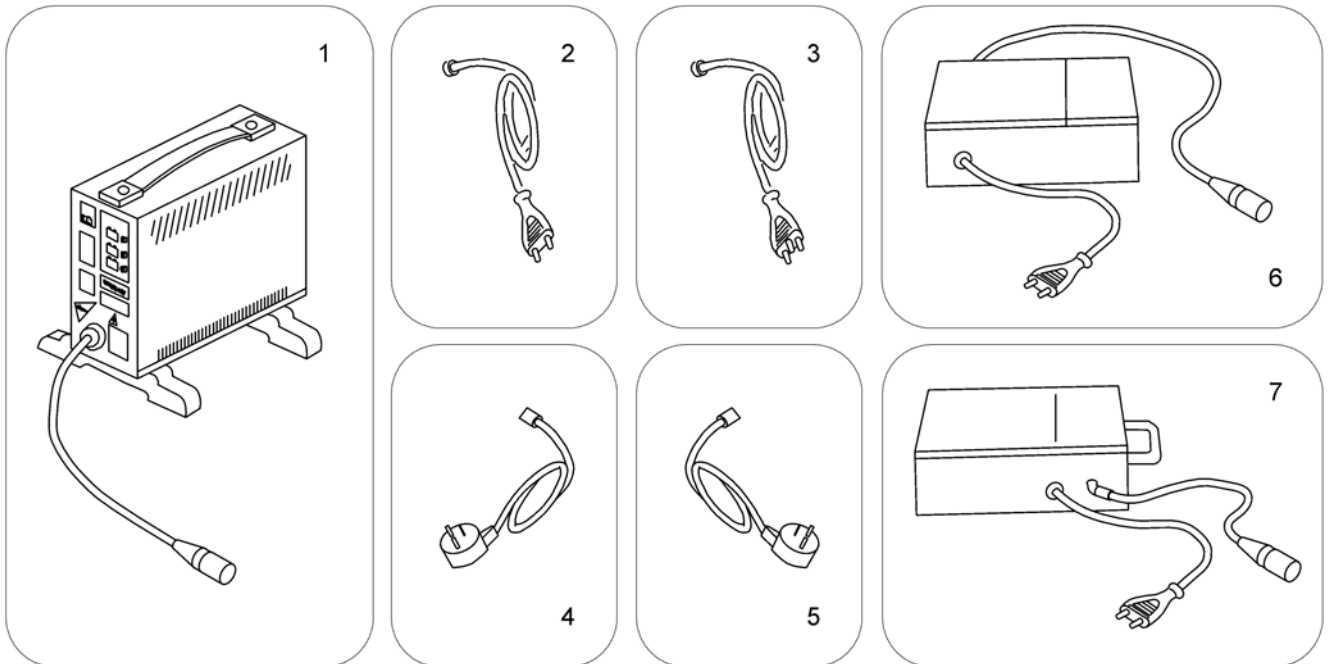


Pos.	Part number	Description	Valid from → until	Qty
1	SPP51981	Charger 8A without Power Cord <i>[1343]</i>		1
2	P21079	Power Cord Continental (Schuko GCB-0401)		1
3	P21080	Charger Cable (UK)		1
4	1443841	Power Cord Swiss (black 6A/250V)		1
5	SP1553849	Power Cord Israel (IEC 320 C13 to SI-32)		1
6	SP1545511	Charger 10A incl. Power Cord Continental <i>[1389]</i>		1
7	5813987	Charger 8A (Protectionclass IP54) incl. Power Cord Continental (Sweden only) <i>[1388]</i>		1

Storm4 TTPlus


Pos.	Part number	Description	Valid from → until	Qty
-		Batteries & Accessories		0
-		Chargers		1

Batteries & Accessories

1581915.iso

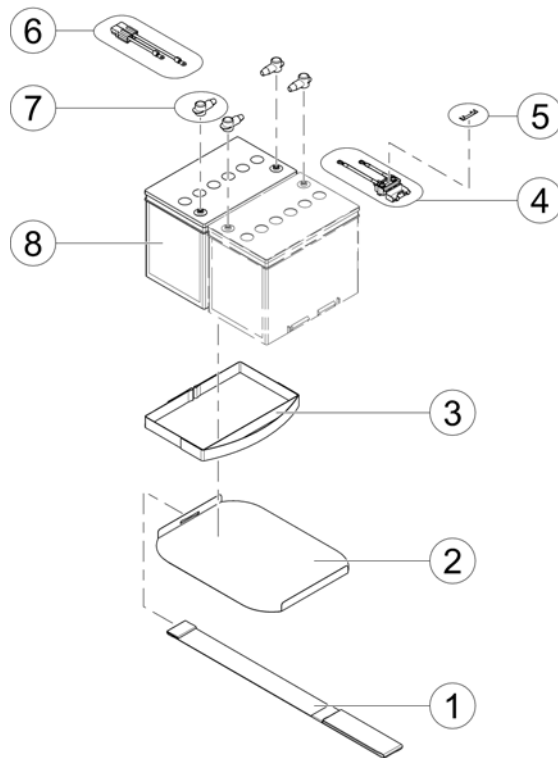


Pos.	Part number	Description	Valid from → until	Qty
-		Important advice! Replace batteries in pairs. Never mix: manufactures, date codes or technologies (Gel with AGM batteries)		1
1	SP1531651	Battery Strap		1
2	SP1524214	Battery carrier sheet		1
3	SP1518073	Battery strap		1
4	SP1579554	Fuse Cable (GGB2)		1
-	SP1588075	Cable Tie Set V0 (10 of each T18R/T30R/T50R)		1
5	1433734	Strip Fuse 80A		1
6	SP1579555	Battery Cable ACS2 (GGB2)		1
7	P52996	Battery terminal cover		4
8	P50016	Battery, 12V/70-73,6 Ah C20, Gel		2

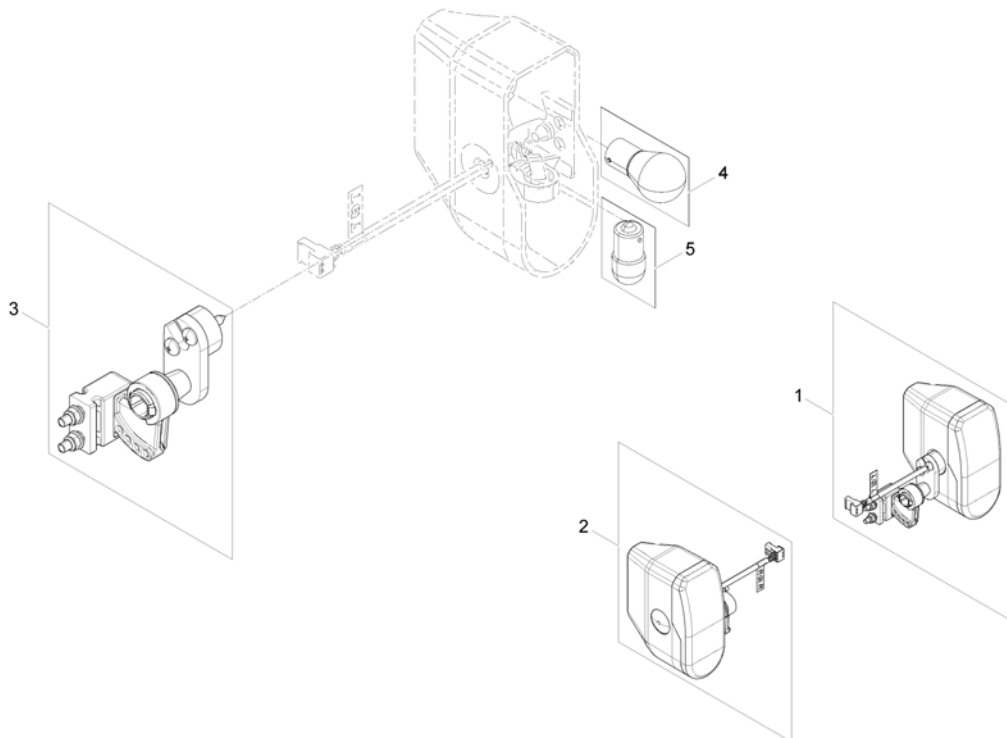

Pos.	Part number	Description	Valid from → until	Qty
1	SP1528319	Frontlight, complete with bracket, LH (Germany)		1
1	SP1528547	Frontlight, without bracket, LH (Germany) <i>not shown</i>		1
2	SP1528320	Frontlight, complete with bracket, RH (Germany)		1
2	SP1528548	Frontlight, without bracket, RH (Germany) <i>not shown</i>		1
3	SP1530571	Frontlight bracket, LH, cpl.		1
3	SP1530570	Frontlight bracket, RH, cpl.		1
4	SP1419271	Light Bulb 24V/21W, yellow (Turn Signal front/rear)		1
5	SPP31108	Light Bulb 24V/10W (BA15s), white (Front)		1


Frontlight for Modulite (German homologation)

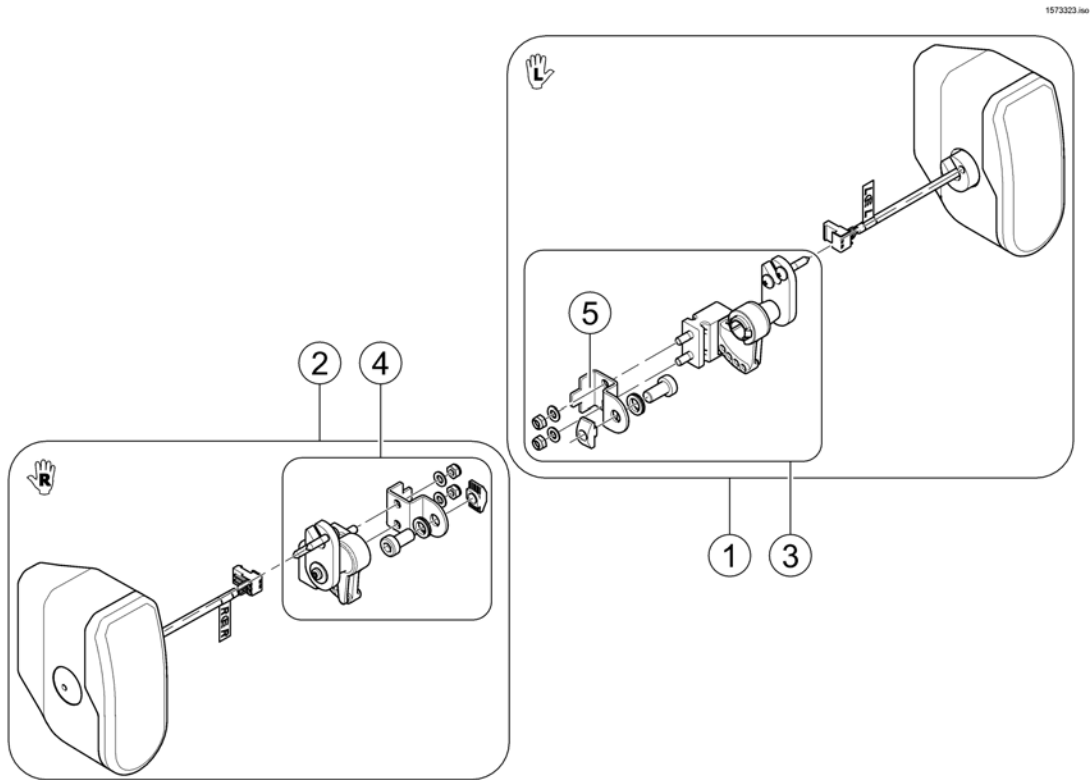

Pos.	Part number	Description	Valid from → until	Qty
-		Bracket design changed on 2015-01-10. For Retrofitting please order SP1556591.		1
1	SP1573237	Frontlight, complete with bracket, LH (Germany)		1
2	SP1573252	Frontlight, complete with bracket, RH (Germany)		1
3	SP1573253	Frontlight bracket, LH, cpl.		1
4	SP1573320	Frontlight bracket, RH, cpl.		1
5	SP1556591	Lamp bracket <i>New design!</i>	2015-01-10 →	2
-	SP1419271	Light Bulb 24V/21W, yellow (Turn Signal front/rear)		1
-	SPP31108	Light Bulb 24V/10W (BA15s), white (Front)		1

Invacare® Storm4 | Safety Options & Accessories
Rearlight (German homologation)


1528769X.dwg

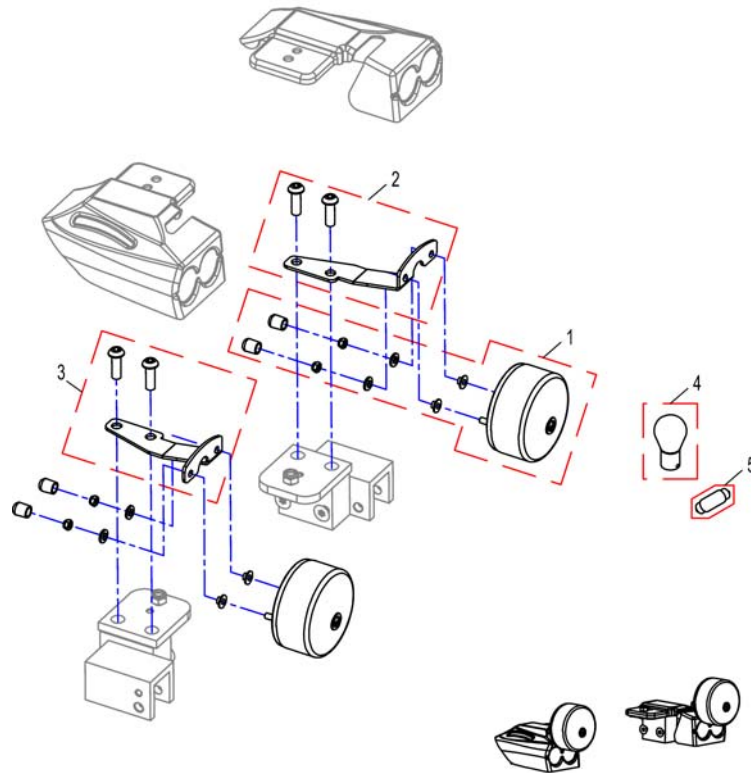

Pos.	Part number	Description	Valid from → until	Qty
!!!	-	Rear Light Protector Kit		0
1	SP1530572	Rearlight, cpl.		1
2	SP1530574	Lampholder back LH		1
3	SP1530573	Lampholder back RH		1
4	SP1419271	Light Bulb 24V/21W, yellow (Turn Signal front/rear)		1
5	P31191	Light Bulb 24V/5W, white (Rear)		1

Rear Light Protector Kit


1585420.00

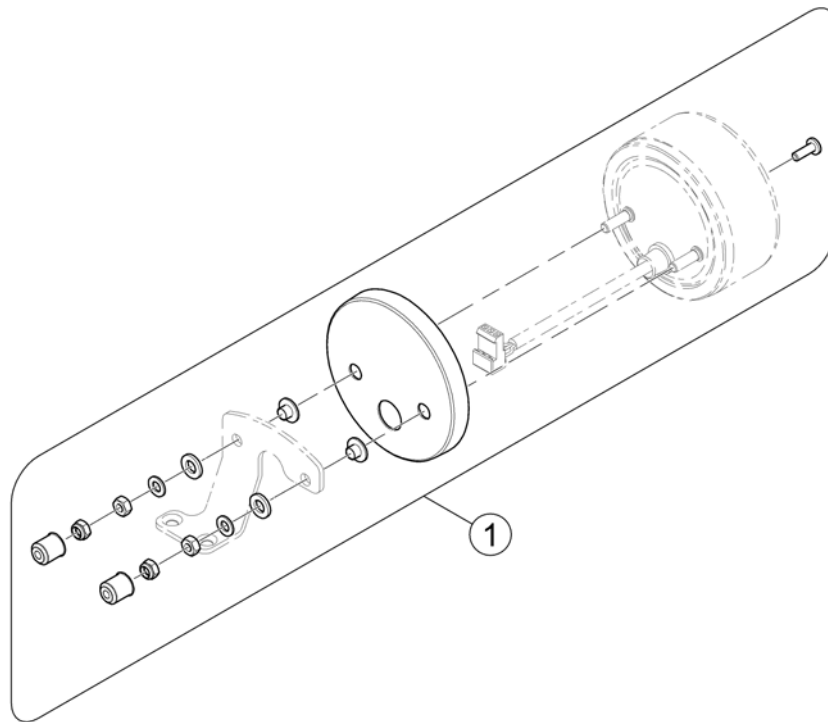

Pos.	Part number	Description	Valid from → until	Qty
1	SP1581827	Rear Light Protector Exchange Kit		1

LED-Frontlight


1527957X.iso

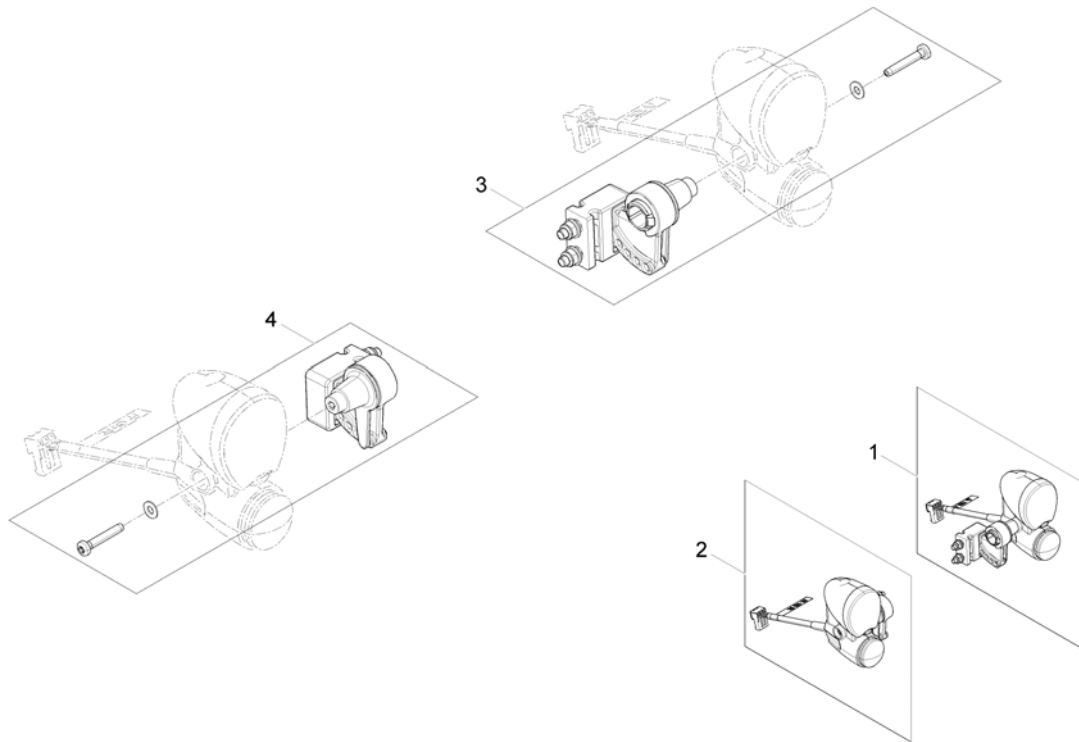

Pos.	Part number	Description	Valid from → until	Qty
-	SP1541713	Adapter cable LED light, LH <i>with ULM Seat only!</i>		1
-	SP1541654	Adapter cable LED light, RH <i>with ULM Seat only!</i>		1
1	SP1527958	LED-Frontlight LH, cpl.		1
2	SP1527957	LED-Frontlight RH, cpl.		1
3	SP1560611	Frontlight bracket LH		1
4	SP1562245	Frontlight bracket RH		1

LED-Frontlight for Modulite


1573222.iss

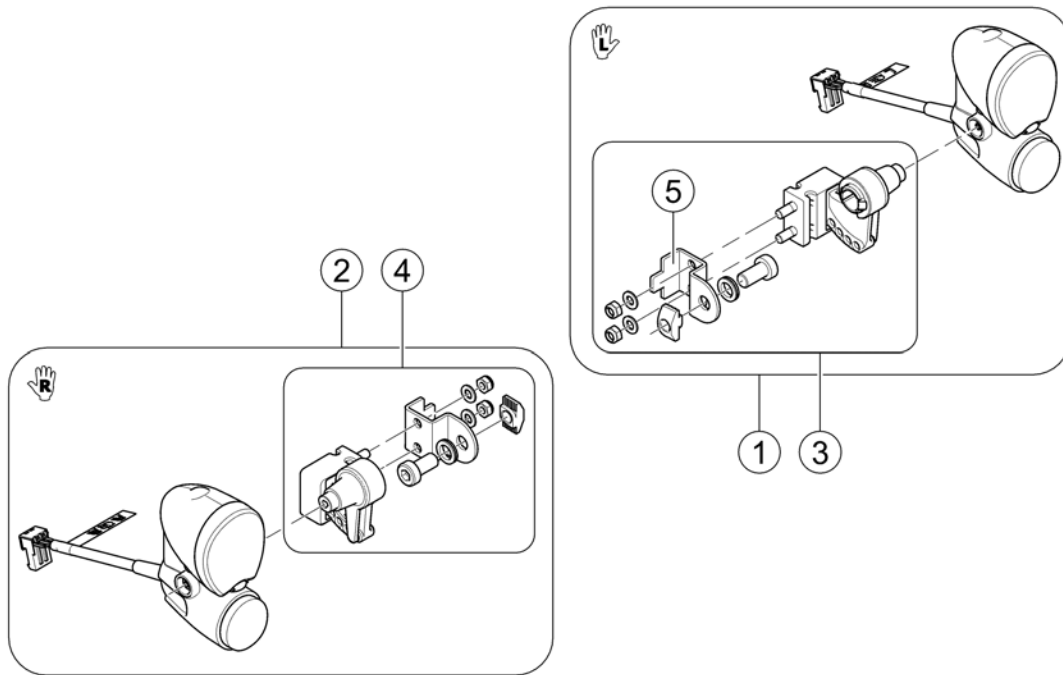

Pos.	Part number	Description	Valid from → until	Qty
-	SP1541713	Adapter cable LED light, LH <i>FDX only!</i>		1
-	SP1541654	Adapter cable LED light, RH <i>FDX only!</i>		1
-		Bracket design changed on 2015-01-10. For Retrofitting please order SP1556591.		1
1	SP1573223	LED-Frontlight LH, cpl.		1
2	SP1573224	LED-Frontlight RH, cpl.		1
3	SP1573226	Frontlight bracket LH		1
4	SP1573227	Frontlight bracket RH		1
5	SP1556591	Lamp bracket <i>New design!</i>	2015-01-10 →	2

LED-Rearlight


15Z914/X.DWG

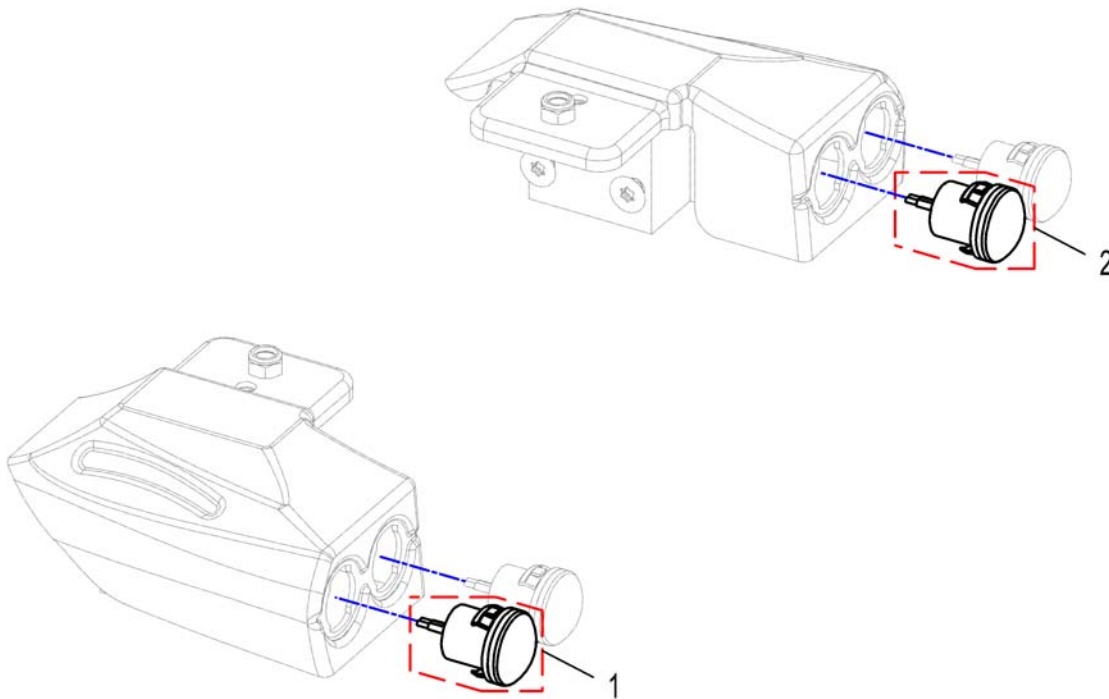

Pos.	Part number	Description	Valid from → until	Qty
-	SP1529712	Rear light LED-Cable		1
1	SP1530287	Turn indicator		1
2	SP1530288	Rear light		1

Attachment kit optimist seat


1567678.iso

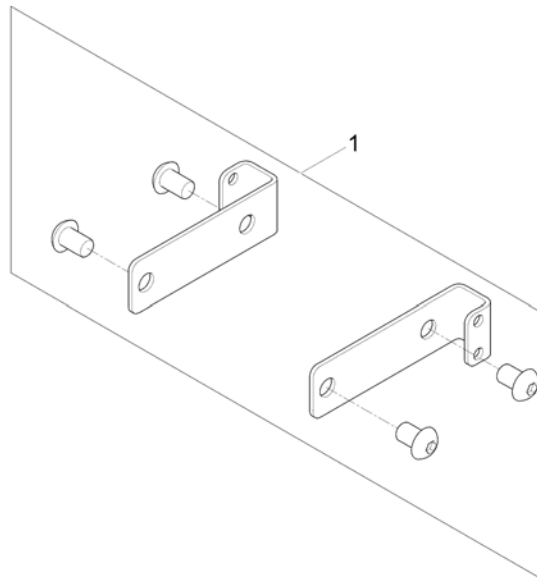

Pos.	Part number	Description	Valid from → until	Qty
	SP1563883	Frontlight bracket Optimist seat		1

Reflective Sign


SP1430410.iss

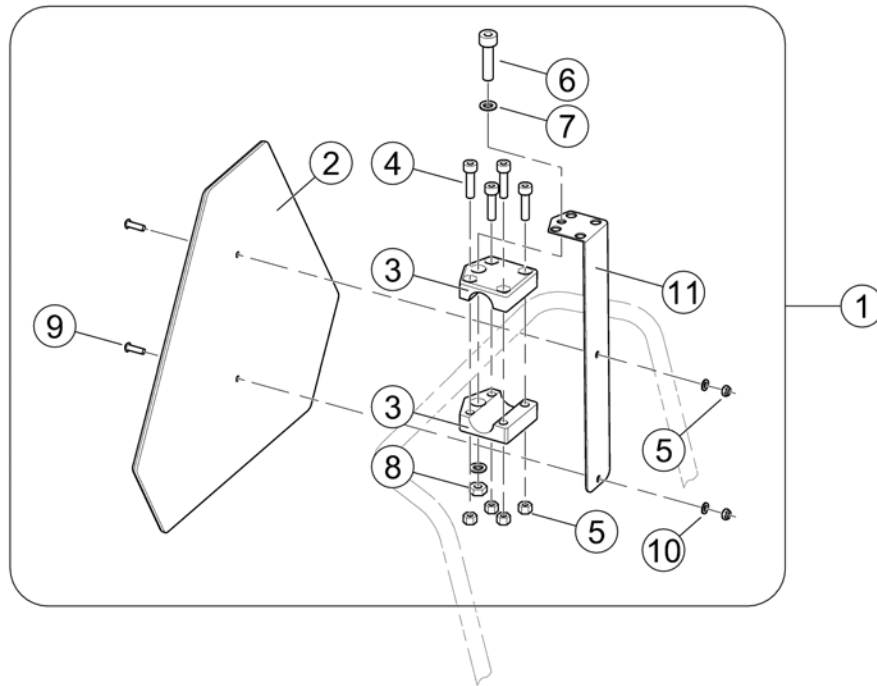

Pos.	Part number	Description	Valid from → until	Qty
1	SP1430410	Reflective Sign ECE-R 69, complete		1
2	1430834	Rear Marking Sign ECE-R 69		1
3	Z12008A.P40	Gripping clip with hexagon hole, RAL 9005 black		2
4	D50004	Cylinder Bolt M6x30-8.8 bk - DIN 912		4
5	F20109	Hexagon Nut M6 bk - DIN 985		6
6	D00102	Hexagon Bolt M8x45-8.8 bk - DIN 931		1
7	G00107	Washer Ø8,4 bk - DIN 125		2
8	F00002	Hexagon Nut M8 bk - DIN 985		1
9	D90187	Fillister Head Screw M6x20-10.9 bk - DIN 7380		2
10	G00115	Washer Ø6,4 bk - DIN 125		2
11	1430418-9005	Mounting Bracket for Reflective Sign - RAL 9005 black		1
12	SP1580671	Bracket, Reflective Sign, cpl.		1

Reflective Sign (Ultra Low Maxx Seat)


1508516.kso

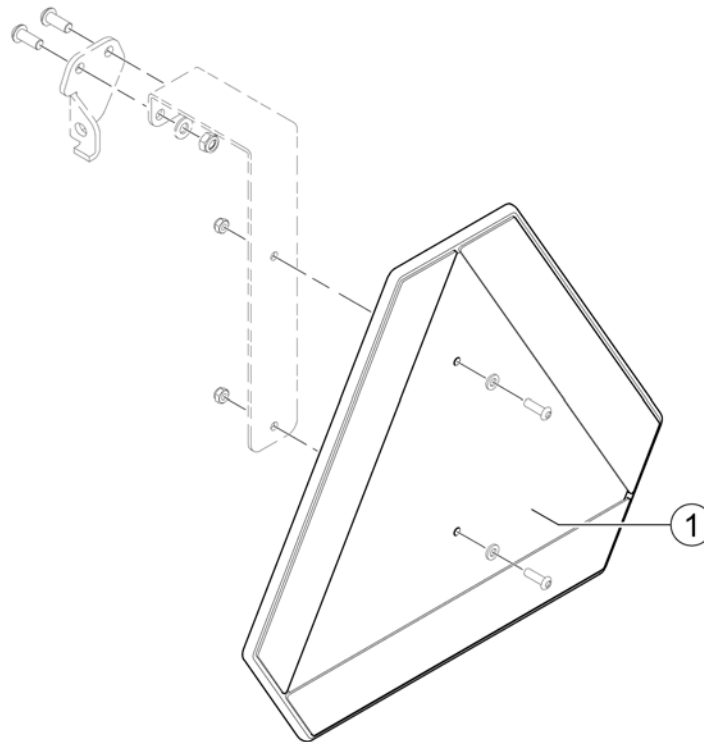

Pos.	Part number	Description	Valid from → until	Qty
2	1430834	Rear Marking Sign ECE-R 69		1
1	1430834	Rear Marking Sign ECE-R 69		1

Kerb Rider


1528952X.dwg

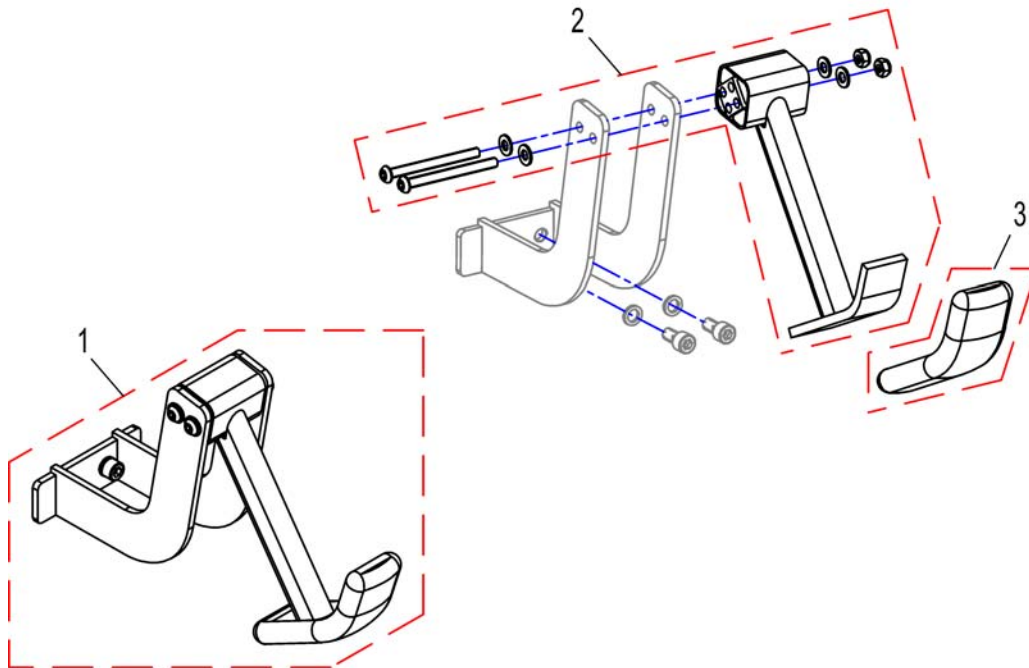

Pos.	Part number	Description	Valid from → until	Qty
1	SP1528952	Kerb Climber cpl. (2-hole connection)	→ 2012-04-12	1
1	SP1551945	Kerb Climber cpl. (3-hole connection)	2012-04-13 →	1
2	SP1530562	Kerb Climber		1
3	B11037B	Rubber Foot		1

Lap Belts / H-Belt


1529460X.dwg

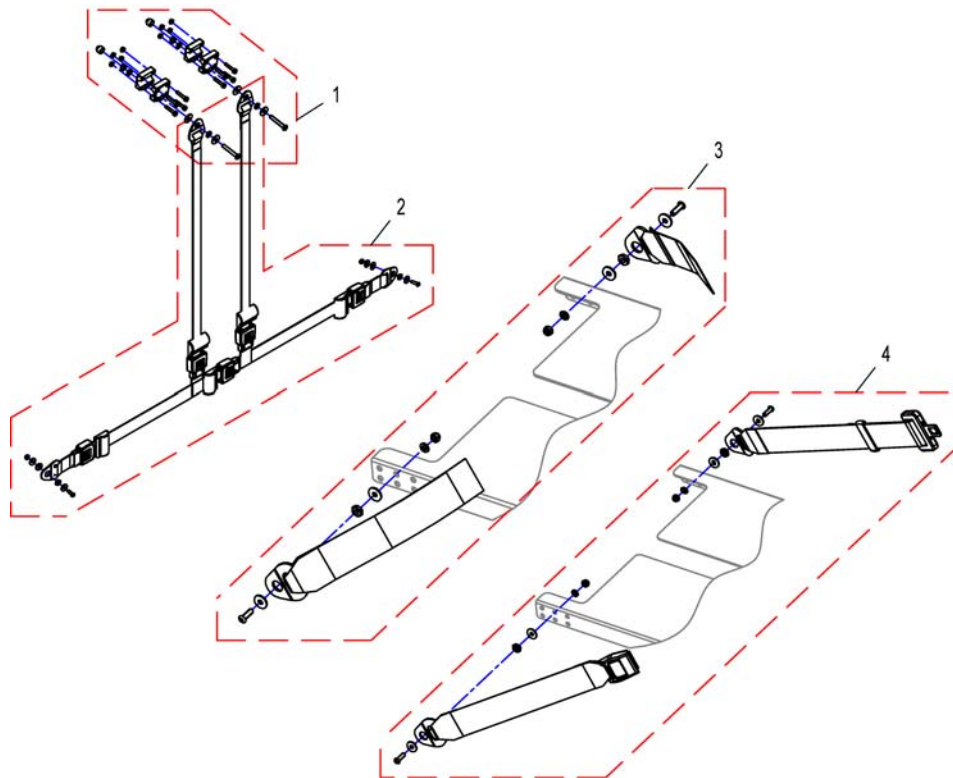

Pos.	Part number	Description	Valid from → until	Qty
1	SP1530575	Mounting-Kit H-Belt		1
2	SP1530576	H-Belt		1
3	SP1529460	Lap Belt Velcro, cpl.		1
4	SP1529461	Lap Belt Autolock, cpl.		1


Lap Belts (Modulite Seat)

1571836 iso

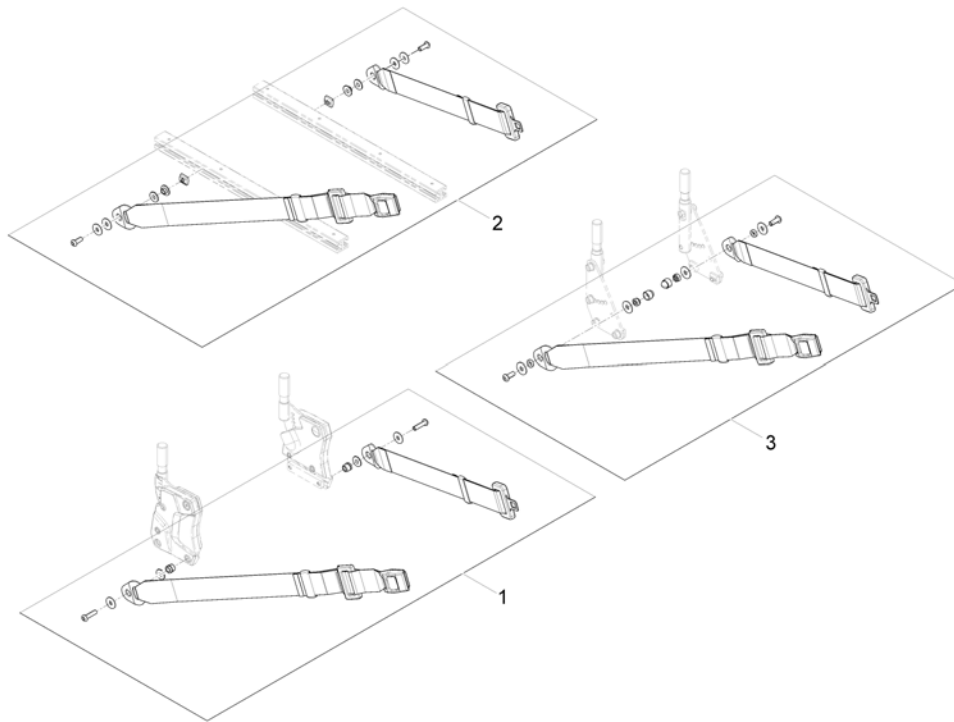

Pos.	Part number	Description	Valid from → until	Qty
1	SP1570381	Lap Belt (fix mounting on Backrest Support), cpl.		1
2	SP1571183	Lap Belt (flexible mounting in side rails), cpl.		1
3	1422845	Lap Belt, cpl.		1


H-Belts (Modulite Seat)

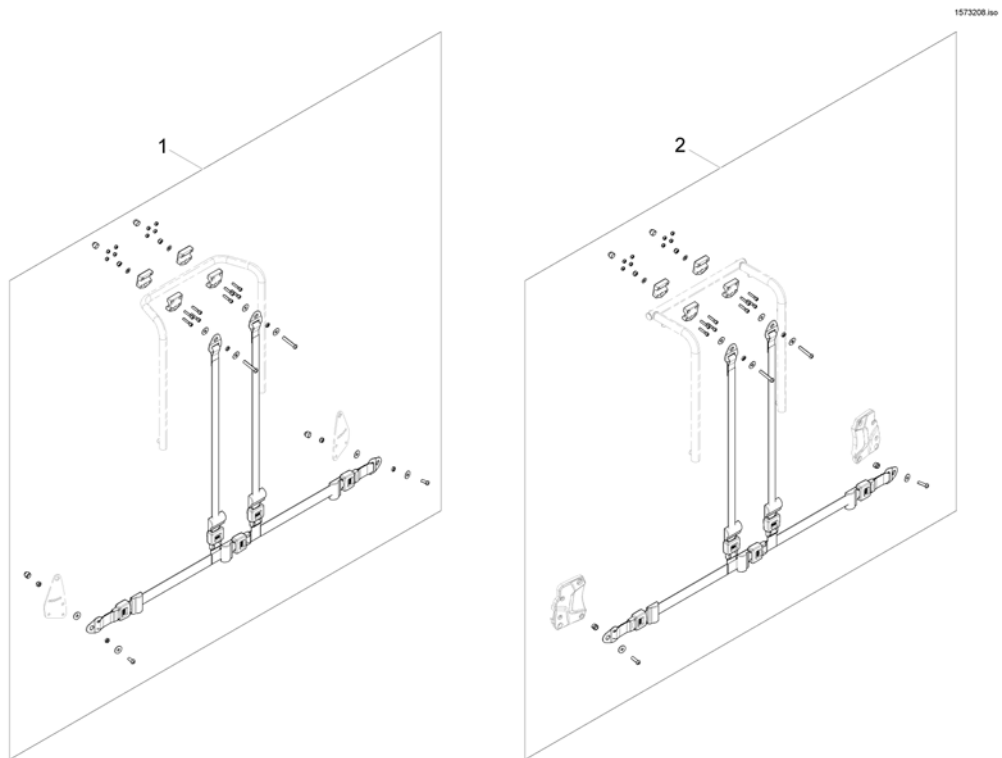

Pos.	Part number	Description	Valid from → until	Qty
1	SP1570385	H-Belt for Backrest Frame		1
2	SP1570380	H-Belt for Back Posts		1

Lap Belt (Ultra Low Maxx)


1588510 iso

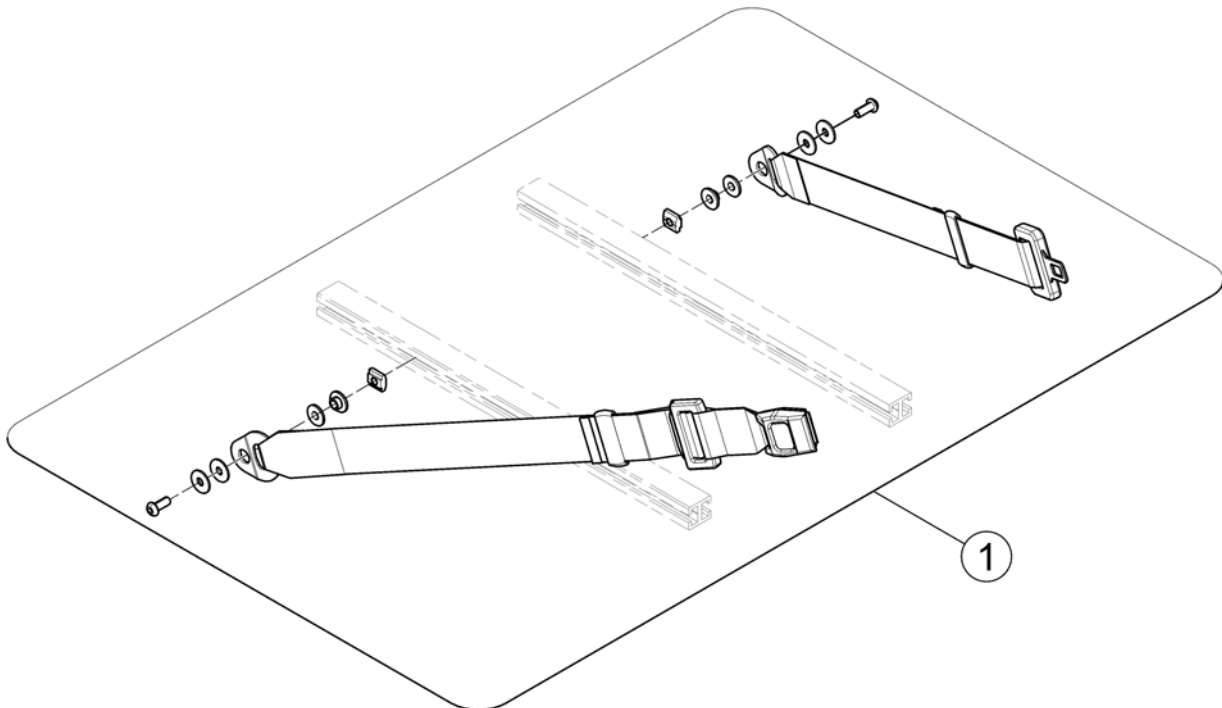

Pos.	Part number	Description	Valid from → until	Qty
1	SP1571183	Lap Belt (flexible mounting in side rails), cpl.		1


Lap Belt (Max Seat)

1582371.00

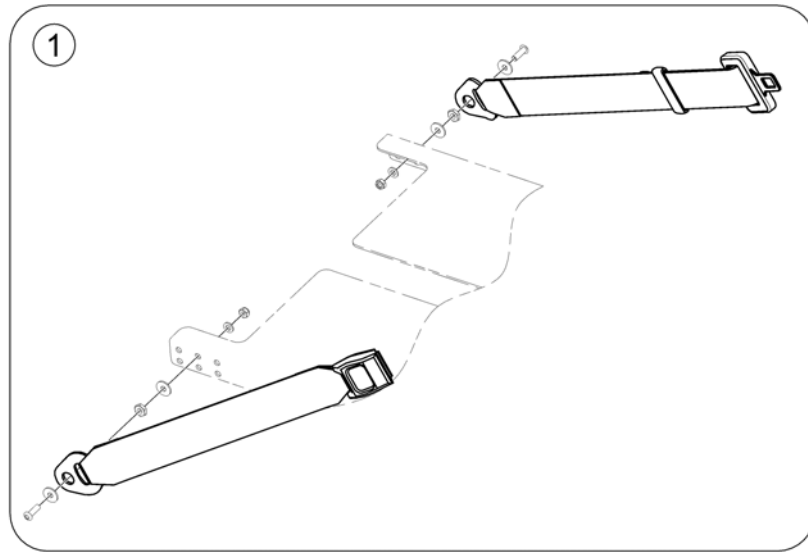

Pos.	Part number	Description	Valid from → until	Qty
1	1551067	Lap Belt Autolock, long		1

Rearview Mirror

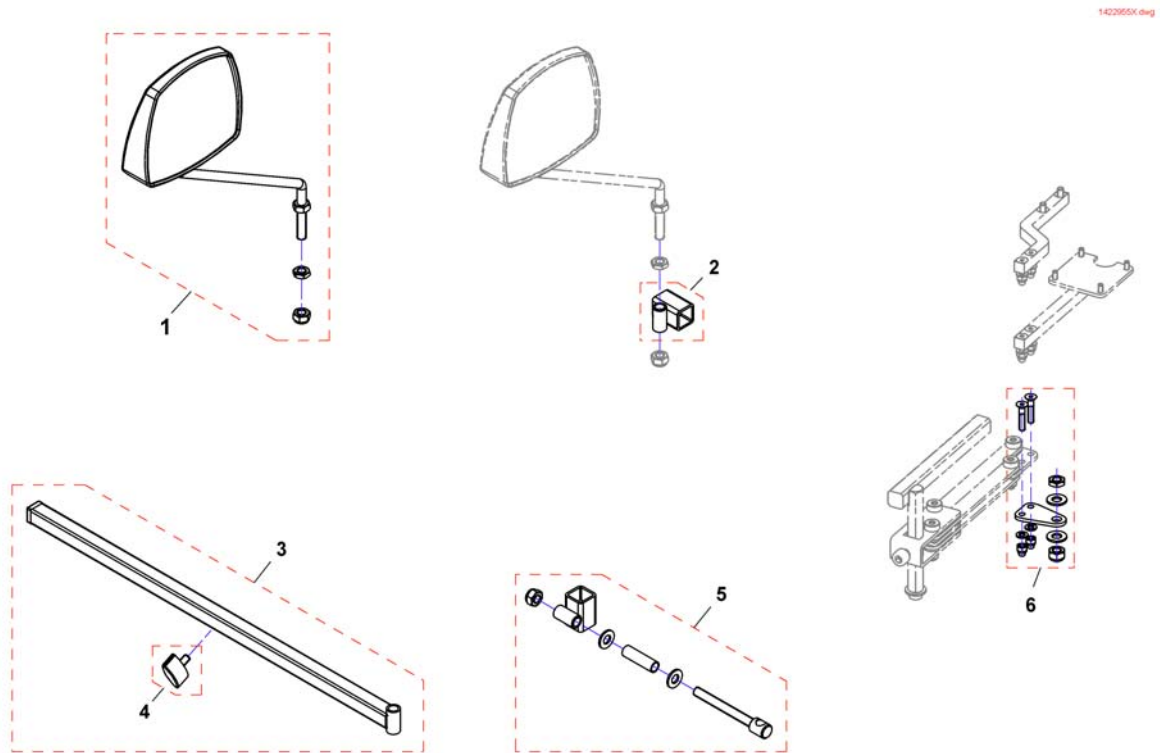

Pos.	Part number	Description	Valid from → until	Qty
1	SP1422955	Rear Mirror		1
2	F276005.00	Mirror bracket		1
3	SP1423435	Rear Mirror Support Rail cpl.		1
4	L31001	Wing ScREW M6X10		1
5	SP1422957	Mirror bracket		1
6	SP1527151	Adapter to mount mirror and remote at the same side		1

Anti-Tippers


Pos.	Part number	Description	Valid from → until	Qty
-		Storm4		0
-		Storm4 TTP		0

Storm4

Pos.	Part number	Description	Valid from → until	Qty
-		Anti-Tipper Wheels => 04.01.11		0
-		Anti-Tipper for SSD Motors		0
-		Anti-Tippers for SSD Motors, flip up		0

Anti-Tipper Wheels => 04.01.11


1428929.dwg

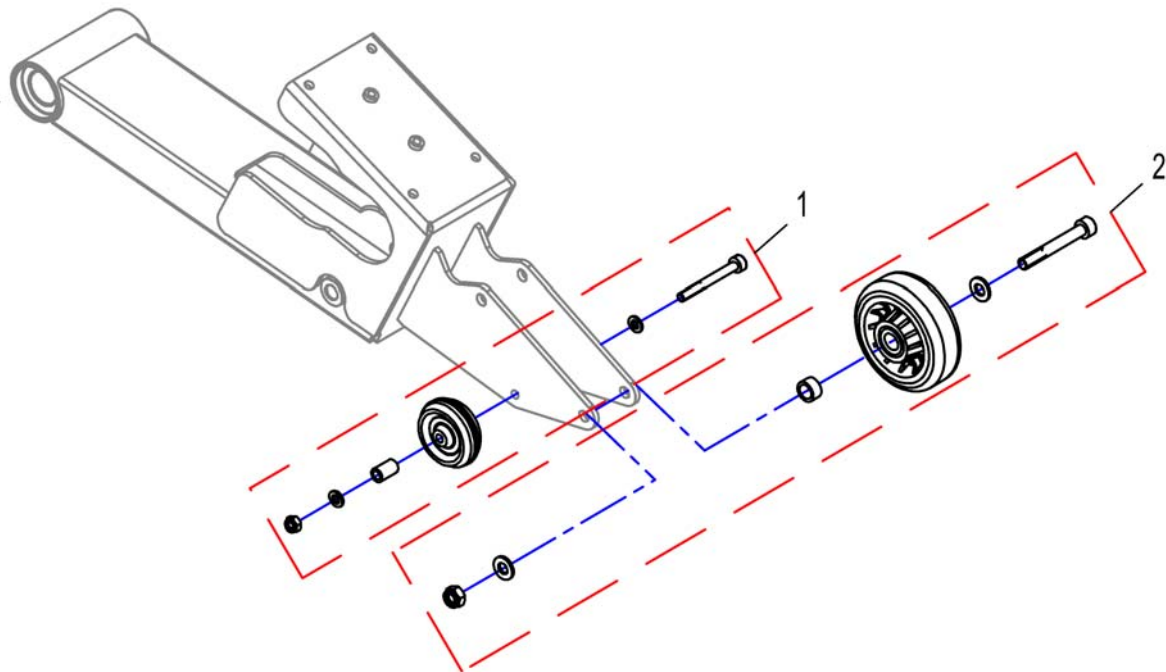

Pos.	Part number	Description	Valid from → until	Qty
1	SP1530555	Anti-Tip-Wheel cpl., Ø50		1
2	SP1530556	Anti-Tip-Wheel cpl., Ø80		1

Anti-Tipper for SSD Motors


1533673X.dwg

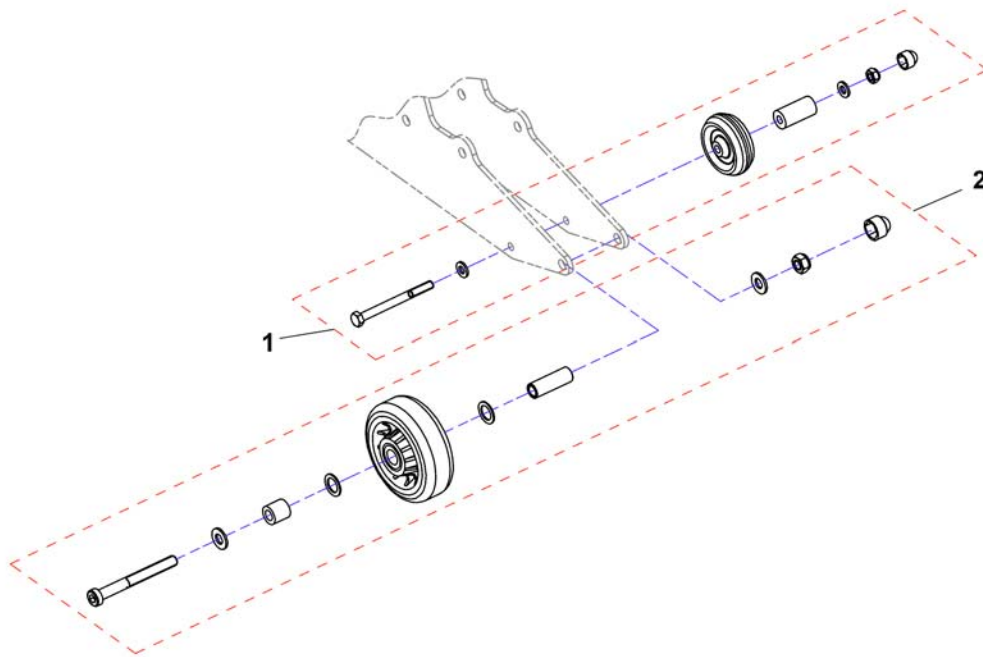

Pos.	Part number	Description	Valid from → until	Qty
1	SP1555703	Anti-Tip-Wheel cpl., Ø50		1
2	SP1555704	Anti-Tip-Wheel cpl., Ø80		1

Anti-Tippers for SSD Motors, flip up

1552985X.dwg

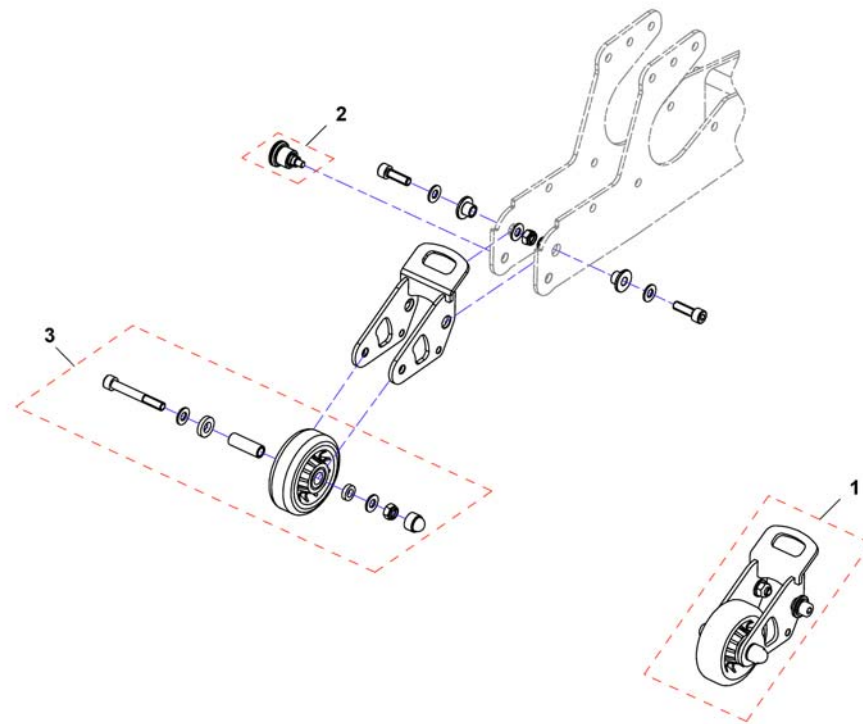


Pos.	Part number	Description	Valid from → until	Qty
1	SP1552420	Anti-Tipper, flip up, cpl.		1
2	SP1550631	Mini indexing plunger		1
3	SP1556002	Anti-Tip-Wheel, Ø80		1

Storm4 TTP

Pos.	Part number	Description	Valid from → until	Qty
-		Anti-Tippers for GGB2 Motors		0

Anti-Tippers for GGB2 Motors

1581916.iso

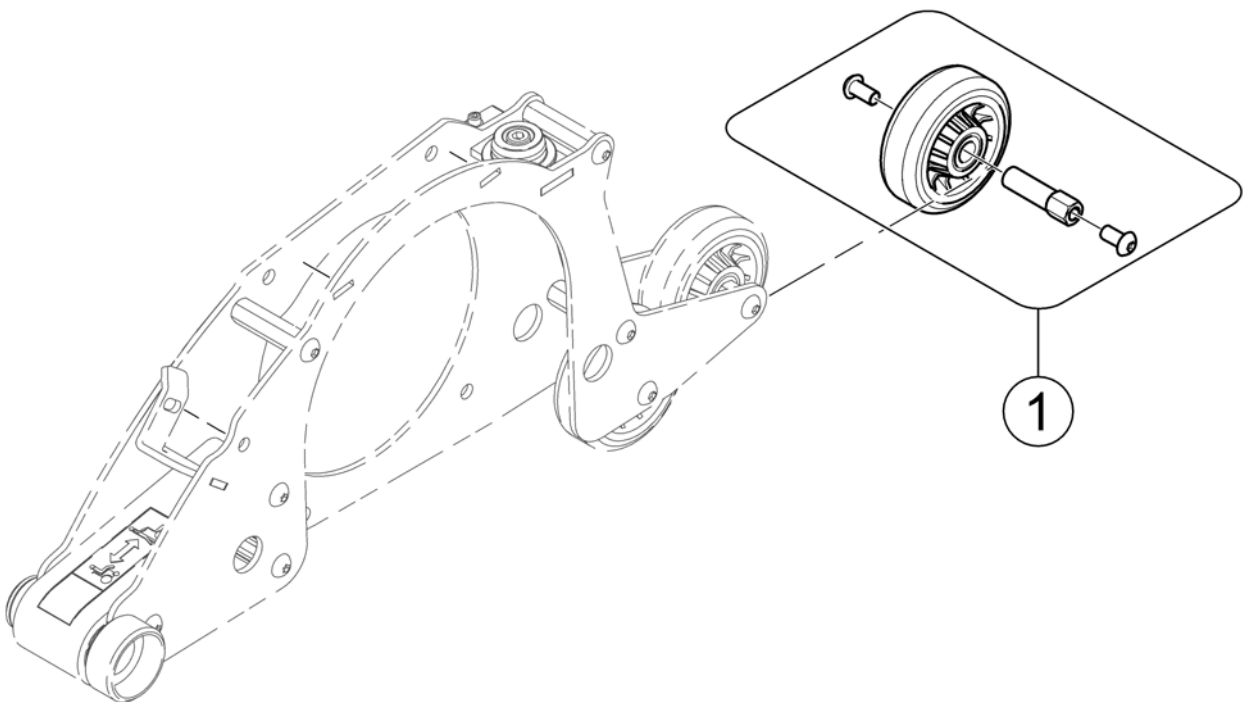


Pos.	Part number	Description	Valid from → until	Qty
1	SP1582215	Anti-Tip-Wheel Kit, Ø80mm		1

Headrests


Pos.	Part number	Description	Valid from → until	Qty
-		Headrest with cheek support [1427]		1
-		Headrest & Bracket [1350]		1
-		Headrestholder EasyAdapt Seat		1
-		Headrest, sidewise adjustable [1432]		1
-		Headrests - Ultra Low Maxx Seat [2020]+[2092/2093/2094]		1

Headrest with cheek support

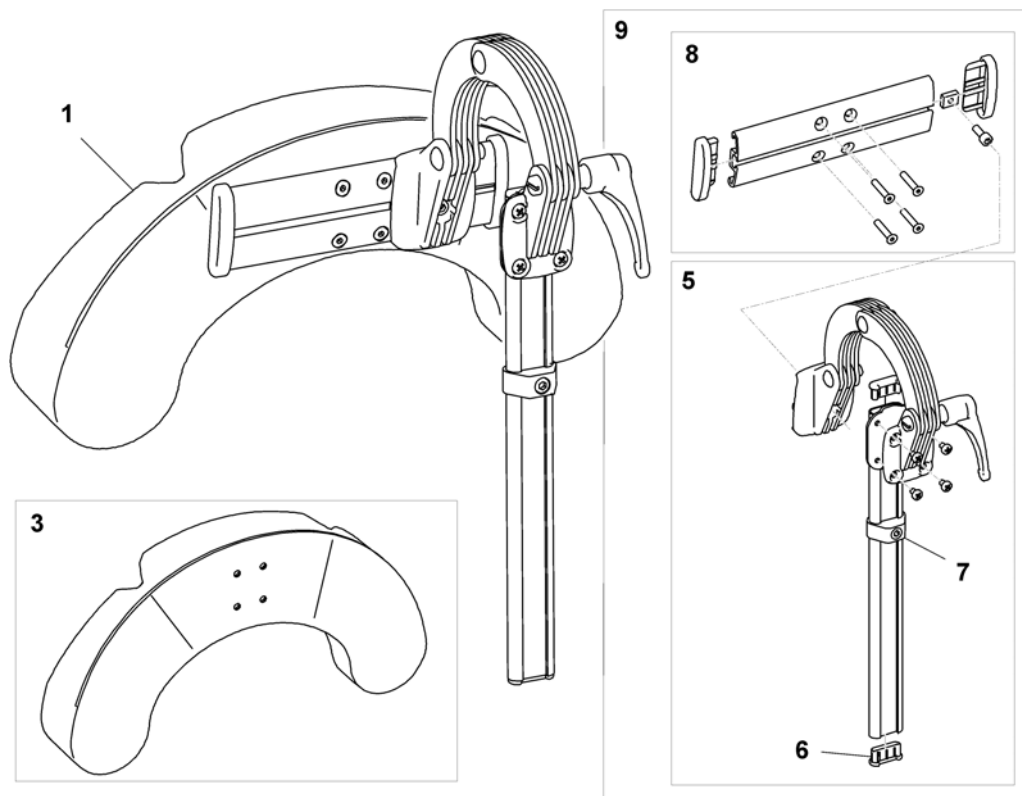

Pos.	Part number	Description	Valid from → until	Qty
1	1446560	HEAD/CHEEK SUPP. TR26 CPL. SIDEWISE ADJUSTABLE		1
1	SP1541658	Head/cheek support TR26 cpl. sidwise adjustable EasyAdapt		1
3	1444304	Cushion, cheek support		1
5	SP1529572	Headrest holder for EasyAdapt		1
6	5716497	Plug IPO "special"		1
7	SP1448661	stop neck and headrest		1
8	1445838	Rail head support		1
9	1485538	Headrest fixation kit		1


Headrest & Bracket

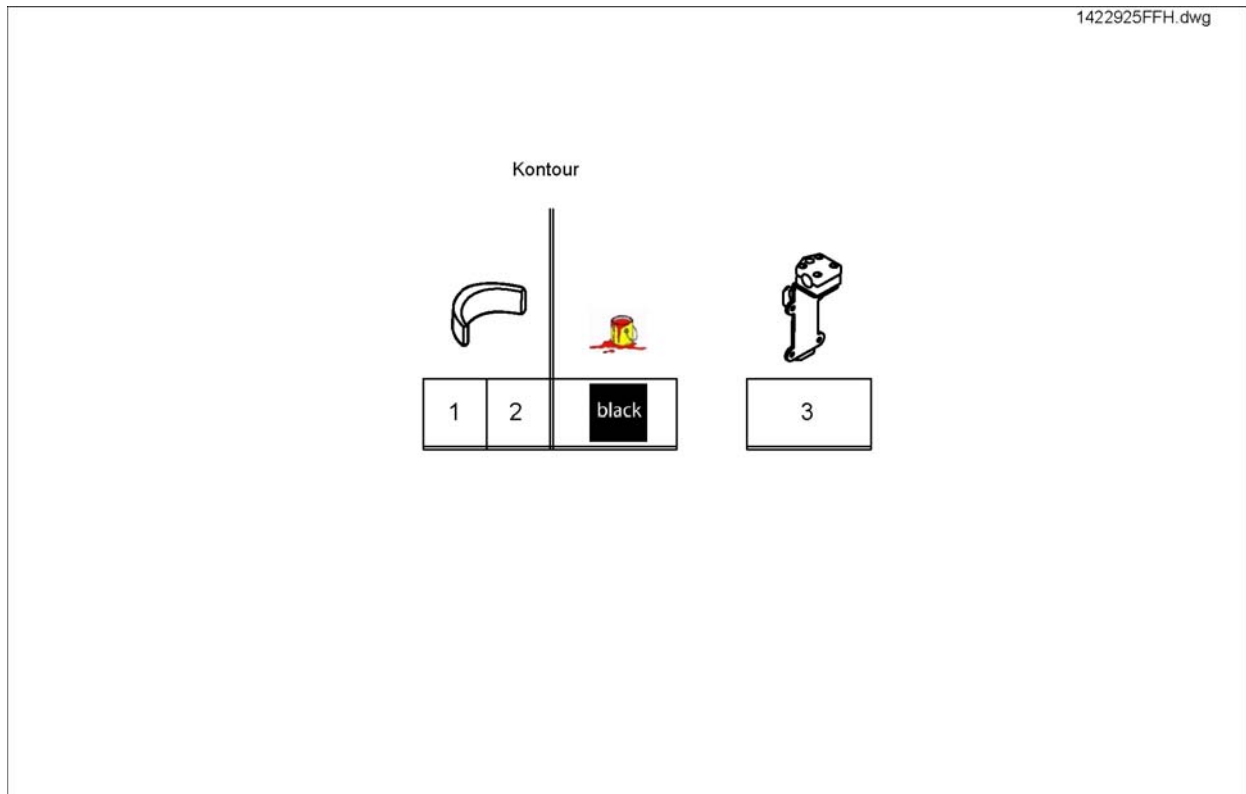

Pos.	Part number	Description	Valid from → until	Qty
1	SP1422925	Headrest, standard/contour seat, black, complete		1
2	SP1422865	HEADREST PAD STD./CONTOUR BLACK		1
3	SP1422914	Head-/Neckrest Bracket, cpl.		1

Headrestholder EasyAdapt Seat


1529463X.dwg

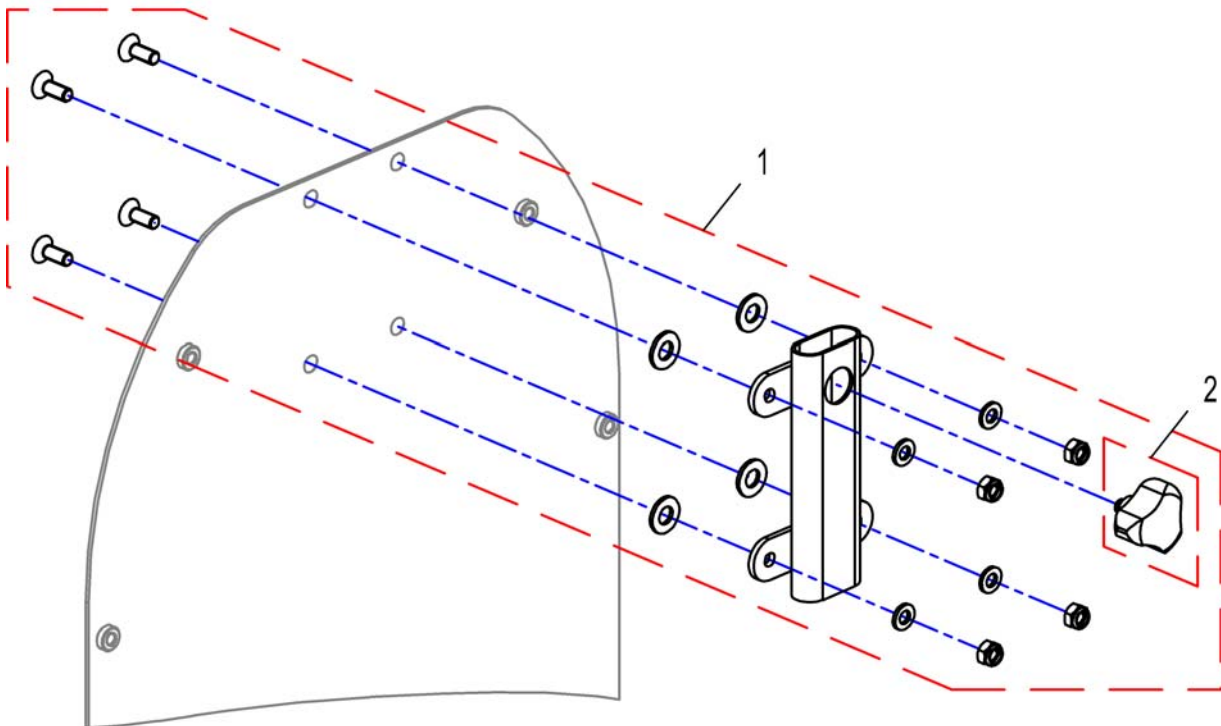

Pos.	Part number	Description	Valid from → until	Qty
1	SP1529463	Headrestholder EasyAdapt Seat, cpl.		1
2	1434623	Ball Screw Court 75mm		1

Headrest, sidewise adjustable


1530911X.no

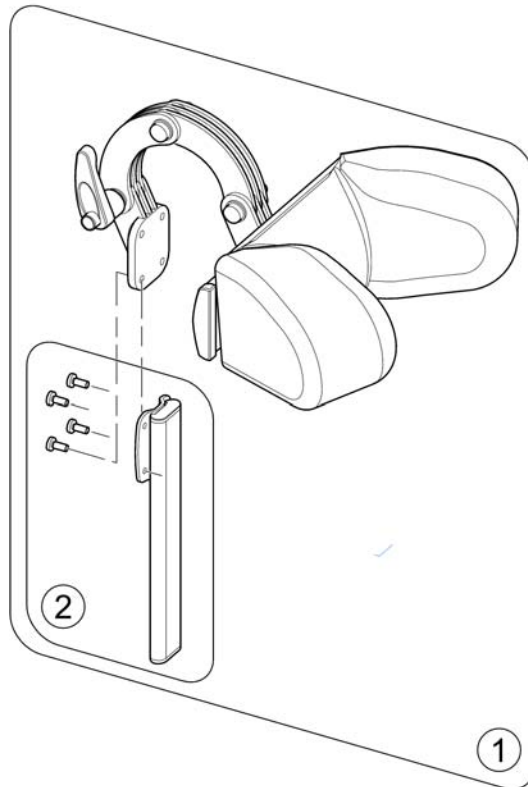

Pos.	Part number	Description	Valid from → until	Qty
1	SP1541657	Headrest, sidewise adjustable, cpl. - for EasyAdapt		1
2	SP1530567	Holder, cpl.		1

Headrests - Ultra Low Maxx Seat

1588517_00

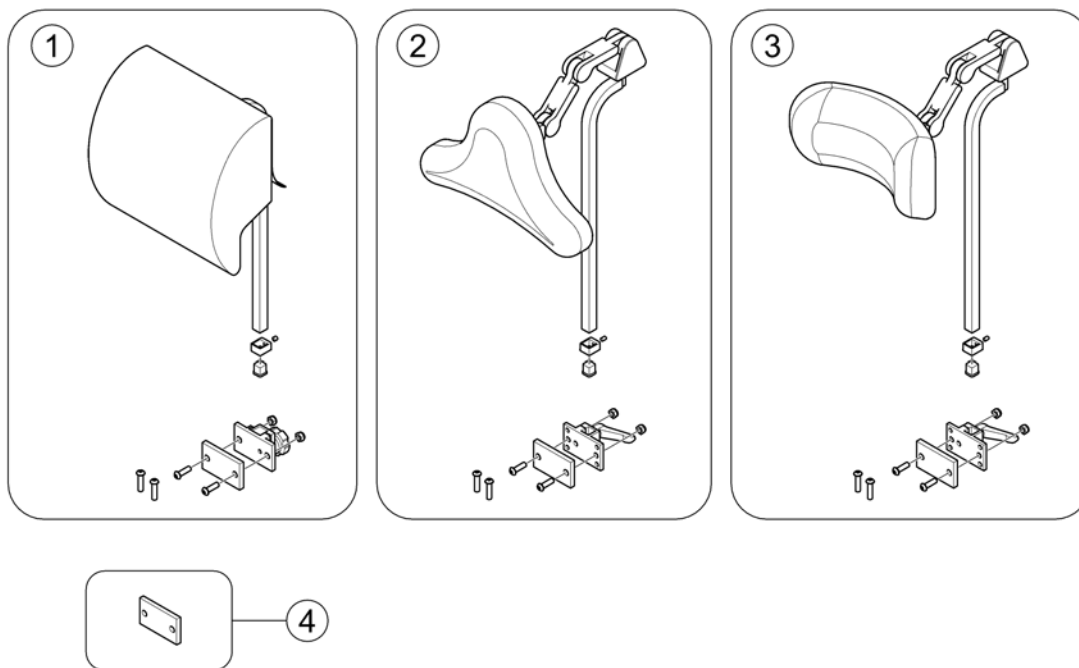


Pos.	Part number	Description	Valid from → until	Qty
1	SPTRC1929	Onyx Auto Style Headrest		1
2	SPTRC1930	All-Position Headrest		1
3	SPTRC1931	Comfort Plus Headrest		1
4	SPTRX2685	Headrest Shim Kit Posture Back		1
4	SP1544188	Headrest Shim Kit Deep Back SW 410-430mm		1
4	SP1544187	Headrest Shim Kit Deep Back SW 460-610mm		1

Lateral Support

Pos.	Part number	Description	Valid from → until	Qty
-		Lateral Supports - Modulite Seat		1
-		Lateral Support - Ultra Low Maxx Seat <i>[2020]+[2043/2044/2045/2046]</i>		1
-		Lateral Support - EasyAdapt Seat		1

Lateral Supports - Modulite Seat


1627167 no

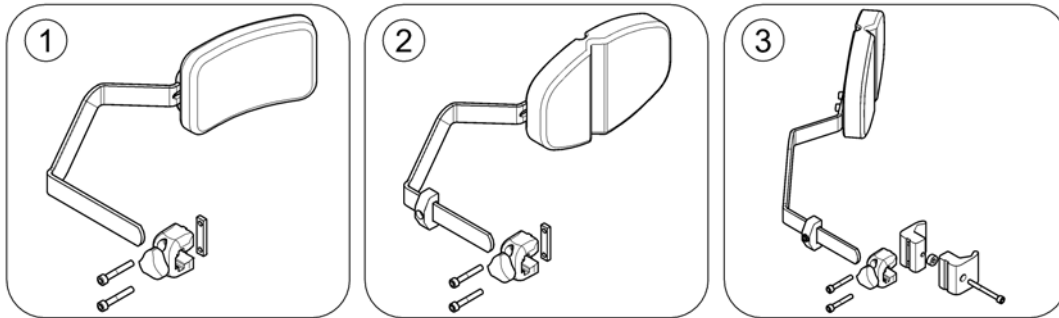

Pos.	Part number	Description	Valid from → until	Qty
1	-	Lateral supports, Flex3 backrest		1
2	-	Lateral supports, Flex3 backrest - multi adjustable		1
3	-	Lateral supports, Flex3 backrest (tension adjustable) - multi adjustable		1

Lateral supports, Flex3 backrest


1007155.00

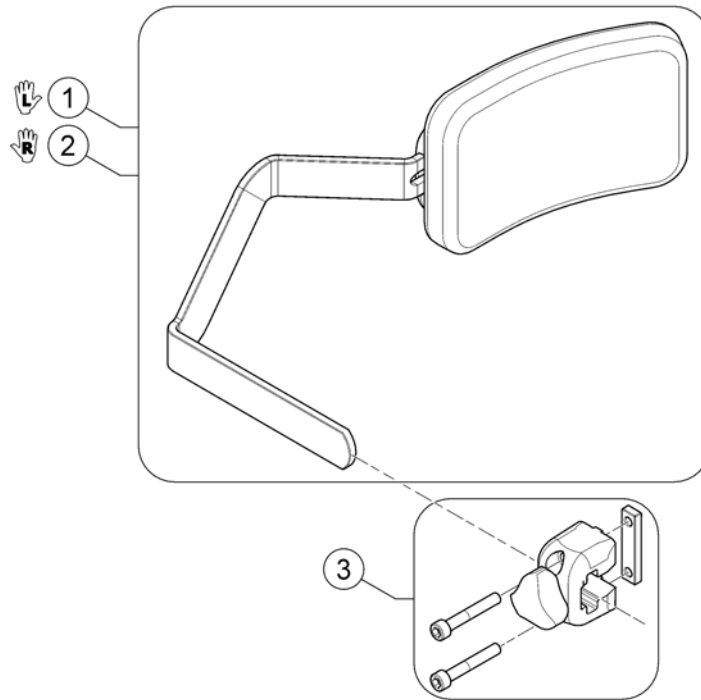

Pos.	Part number	Description	Valid from → until	Qty
-	-	Design change on 01. July 2016 - we suggest to change older supports on both sides.		1
1	SP1552678	Lateral support extended, cpl. - left		1
2	SP1552679	Lateral support extended, cpl. - right		1
3	1543574	Locking trunk support		1

Lateral supports, Flex3 backrest - multi adjustable


160205.00

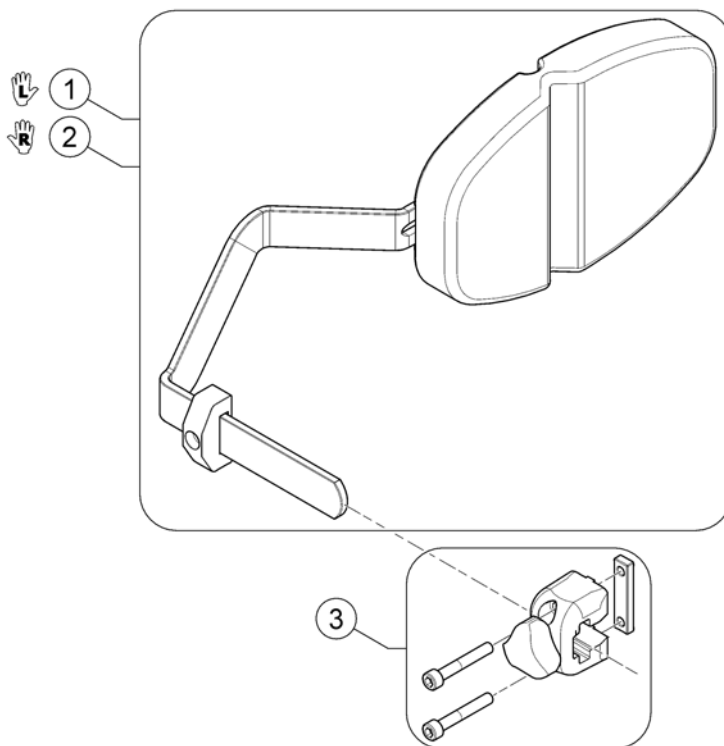

Pos.	Part number	Description	Valid from → until	Qty
-	-	Design change on 01. July 2016 - we suggest to change older supports on both sides.		1
1	SP1555692	Lateral support, multi-adjustable, cpl. - left		1
2	SP1555691	Lateral support, multi-adjustable, cpl. - right		1
3	1543574	Locking trunk support		1

Lateral supports, Flex3 backrest (tension adjustable) - multi adjustable


1607219 iso

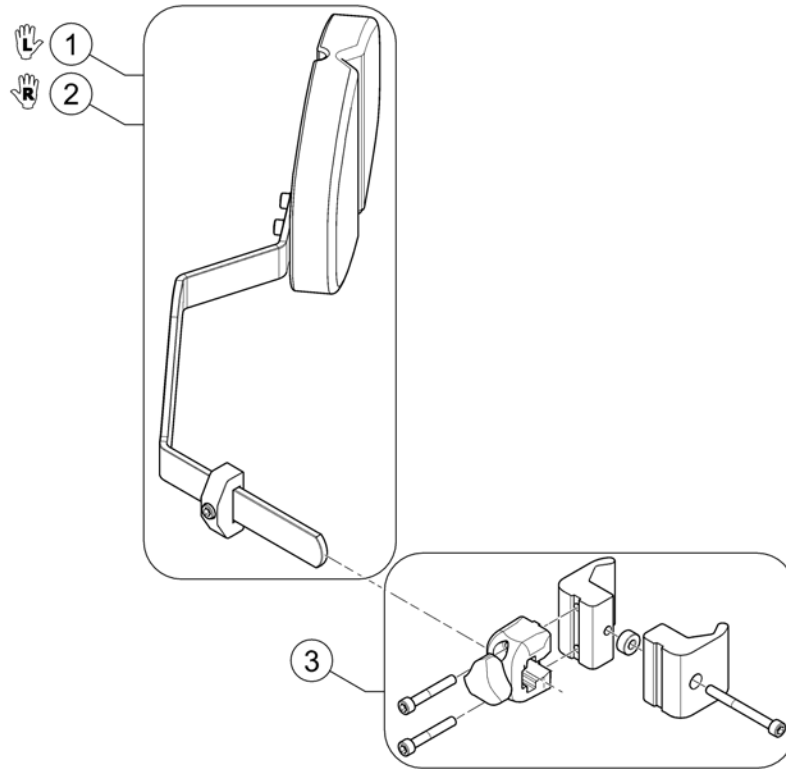

Pos.	Part number	Description	Valid from → until	Qty
1	SP1562920	Lateral Support, Flex3 back, tension adj., extended, multi-adj. - LH		1
2	SP1562919	Lateral Support, Flex3 back, tension adj., extended, multi-adj. - RH		1
3	SP1554143	Mounting hardware Lateral Supports, Flex3 back, tension adj.		1

Lateral Support - Ultra Low Maxx Seat

1588596.iss

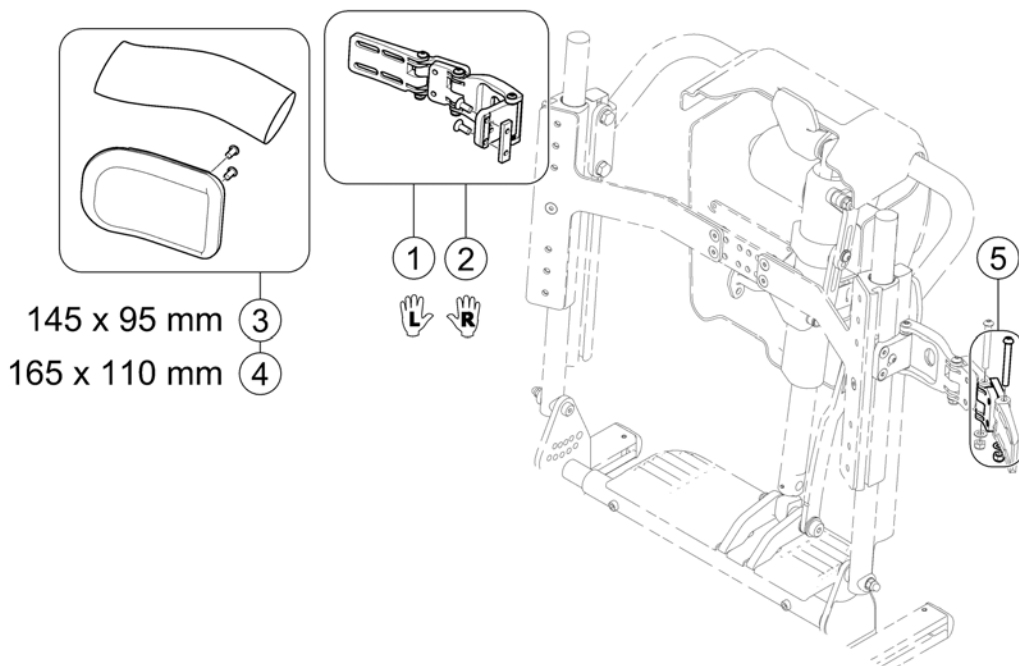


Pos.	Part number	Description	Valid from → until	Qty
1	SPTRC1806	Lateral Hardware Assy LH (w/o sleeve)		1
2	SPTRC1805	Lateral Hardware Assy RH (w/o sleeve)		1
3	SPTRC2123-C03	Lateral Pad Kit 145x95 mm & sleeve for hardware		1
4	SPTRC2123-C04	Lateral Pad Kit 165x110 mm & sleeve for hardware		1
5	SPTRC2091	Extension Kit, Lateral Support (Set of 2 pcs) incl. longer sleeves		1

Lateral Support - EasyAdapt Seat

Pos.	Part number	Description	Valid from → until	Qty
1	-	Lateral Support (Dartex) [1446]		1
2	-	Lateral Support, multi adjustable with flexible pad [1445]		1

Lateral Support (Dartex)


1570269.000

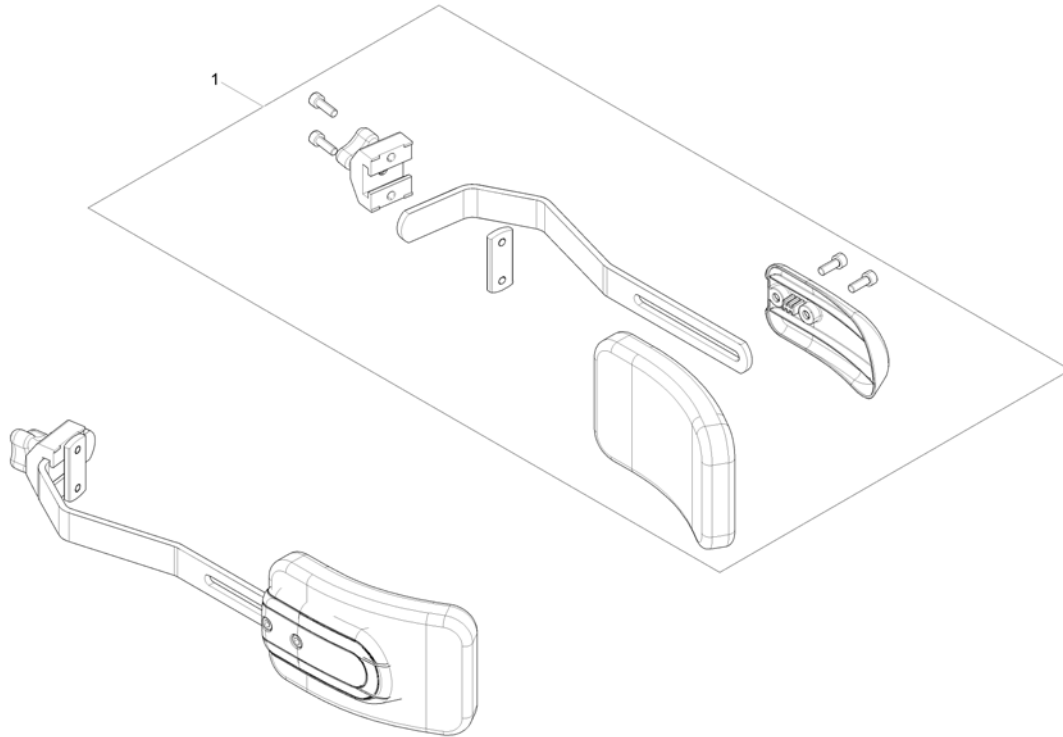

Pos.	Part number	Description	Valid from → until	Qty
	SP1561878	Trunk support, cpl.		1

Lateral Support, multi adjustable with flexible pad


1486065 iso

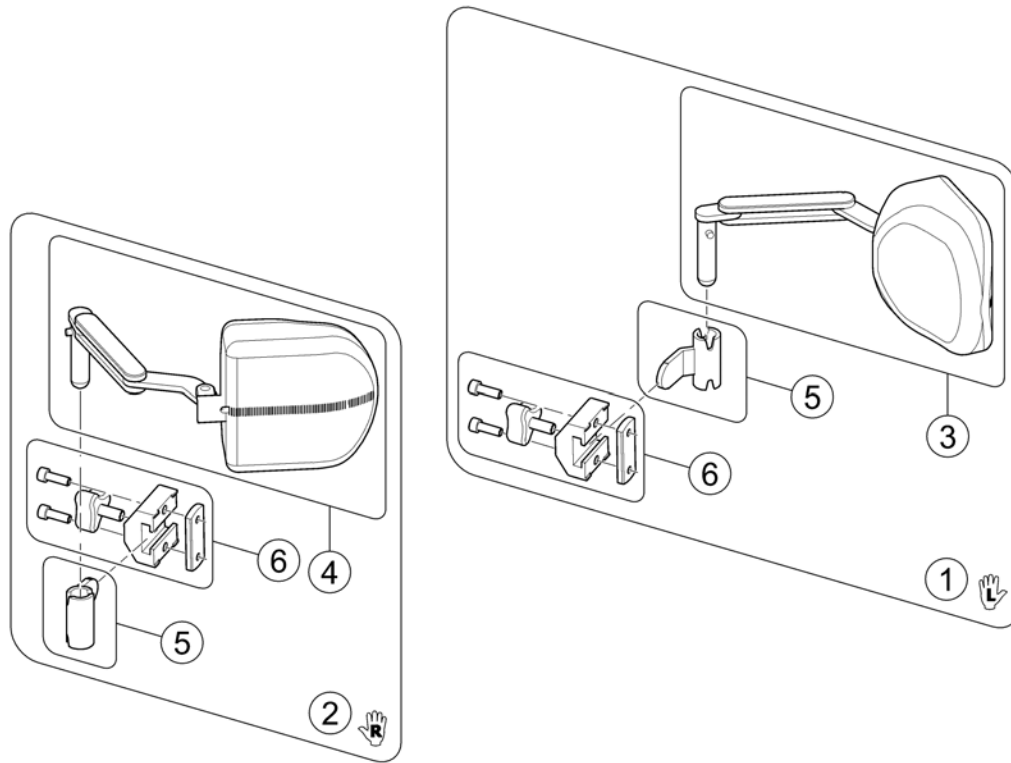

Pos.	Part number	Description	Valid from → until	Qty
1	SP1486065	Lateral Support, cpl. - LH		1
2	SP1486066	Lateral Support, cpl. - RH		1
3	1451822	Lateral support, multi adjustable - LH		1
4	1451821	Lateral support, multi adjustable - RH		1
5	1524903	Bracket LH/RH		1
6	5253170	Lateral Support Bracket		1

Bag


1536178X.dwg

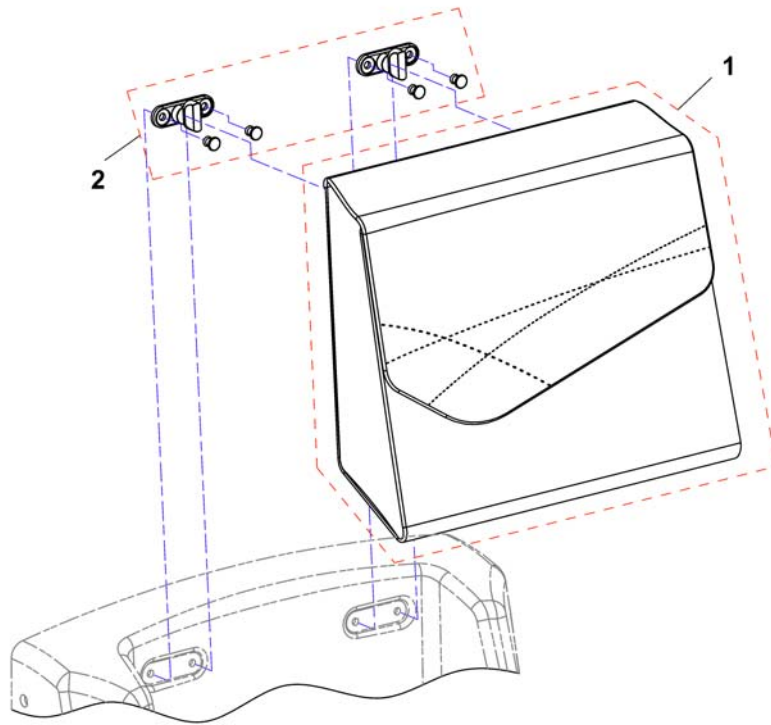

Pos.	Part number	Description	Valid from → until	Qty
1	SP1536178	Bag		1
2	SP1537708	Mounting-Kit		1

Luggage Rack

1551497X.dwg

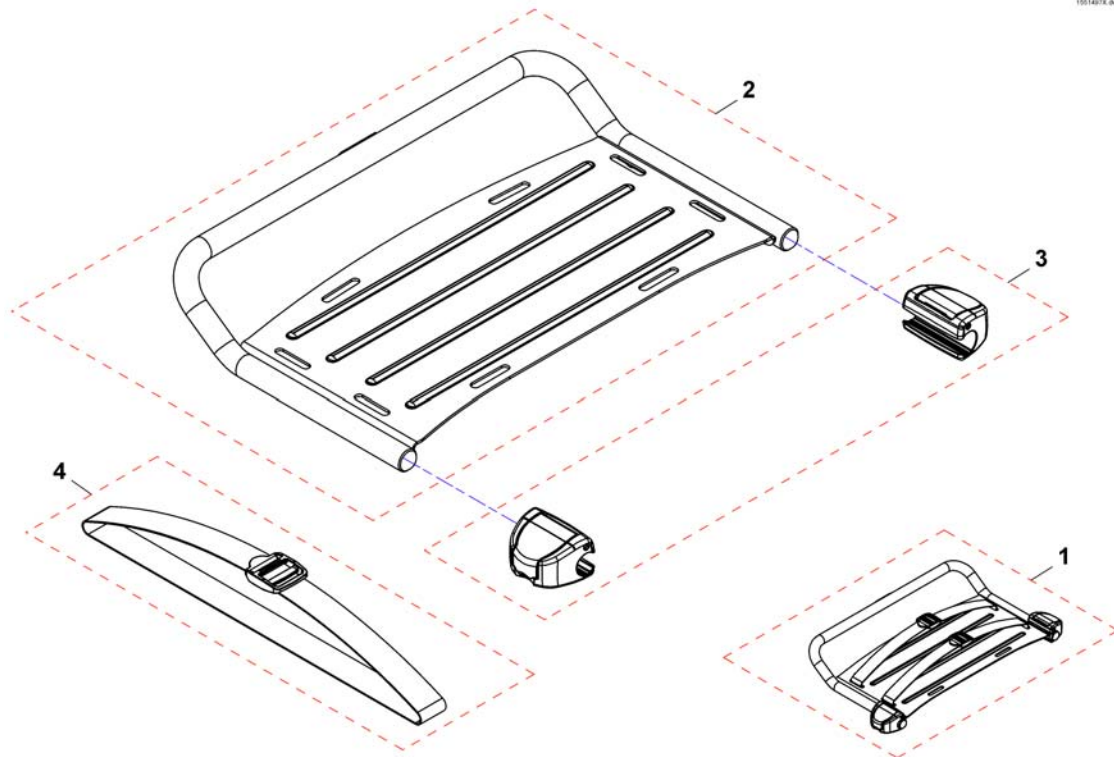


Only for chairs without lights, or with LED lights.

Pos.	Part number	Description	Valid from → until	Qty
1	SP1551619	Luggage rack cpl.		1
2	SP1551497	Luggage rack		1
3	SP1556240	Luggage rack bracket		1
4	F384020	Lashing strap		1

Mounting Kit Luggage Rack

1546314X.dwg

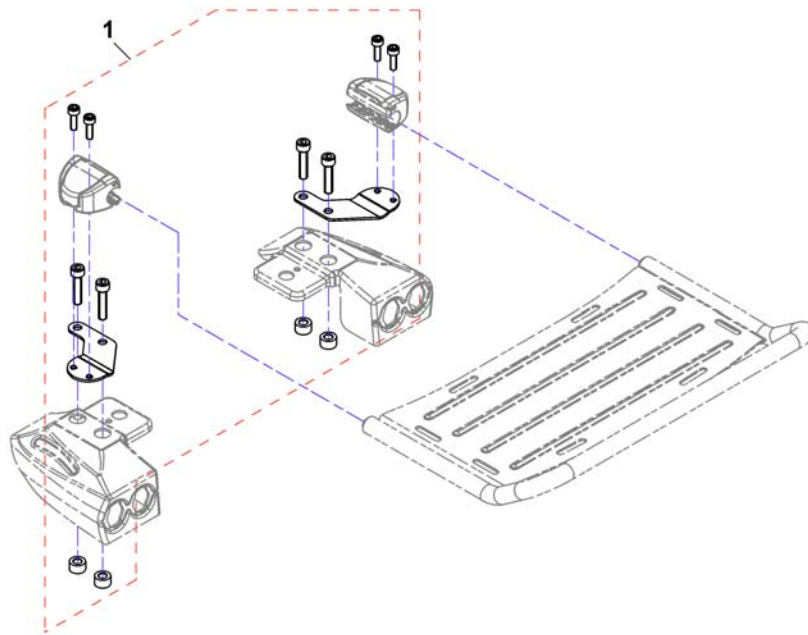


Pos.	Part number	Description	Valid from → until	Qty
1	SP1546314	mounting kit luggage rack Storm4		1

Caneholder

Pos.	Part number	Description	Valid from → until	Qty
-		Cane holder (Easy Adapt Seat & Std-Seat)		1
-		Cane holder (Recaro Seat)		1
-		Cane holder (Modulite: backrest frame)		1
-		Cane holder (Modulite: back posts man recliner)		1
-		Cane holder (Modulite: back posts pow recliner)		1

Cane holder (Easy Adapt Seat & Std-Seat)


1567015.iso

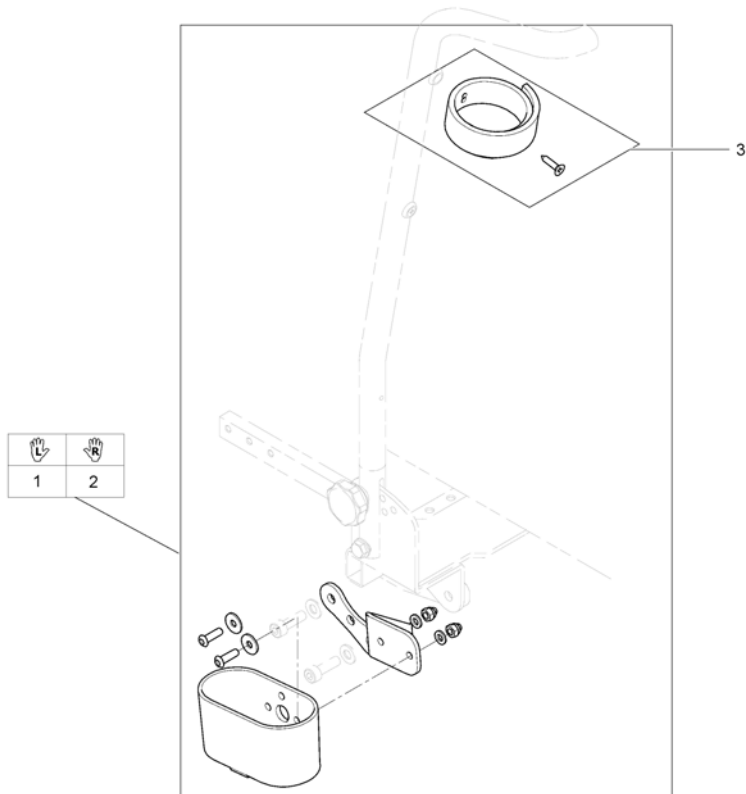

Pos.	Part number	Description	Valid from → until	Qty
1	SP1528444	Caneholder,general, cpl.		1
2	SP1529004	Caneholder,general, cpl.		1
3	SP1420493	Belt for caneholder		1

Cane holder (Recaro Seat)


1567145.iso

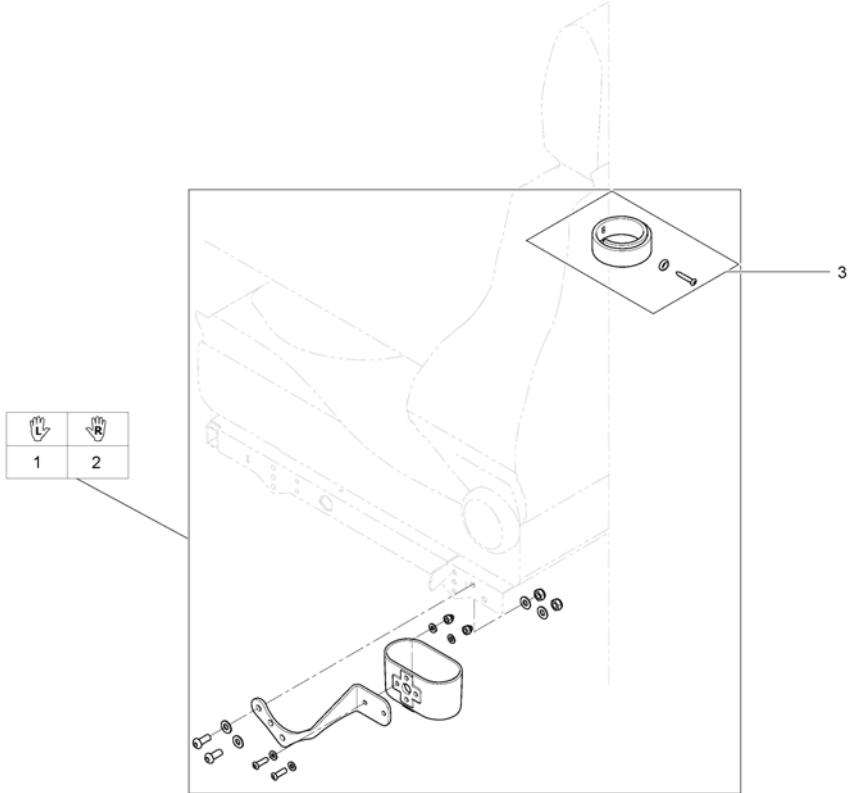

Pos.	Part number	Description	Valid from → until	Qty
1	SP1529535	Caneholder Recaro cpl		1
2	SP1529256	Caneholder Recaro cpl		1
3	SP1420493	Belt for caneholder		1

Cane holder (Modulite: backrest frame)


1571962.jpg

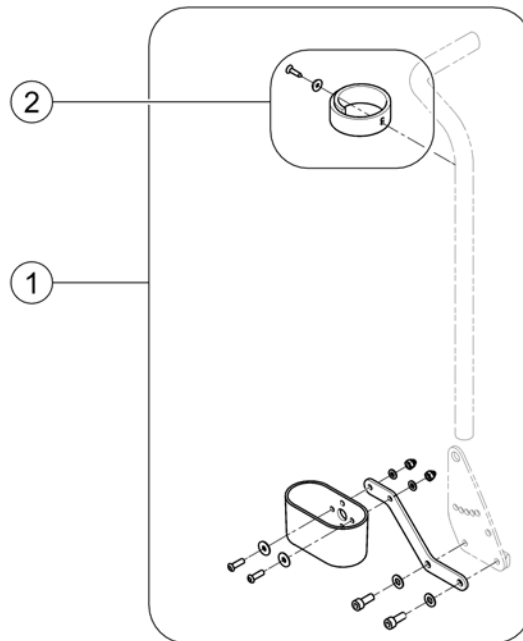

Pos.	Part number	Description	Valid from → until	Qty
1	SP1570396	Cane holder, standard backrest		1
2	SP1420493	Belt for caneholder		1

Cane holder (Modulite: back posts man recliner)


1571903.00

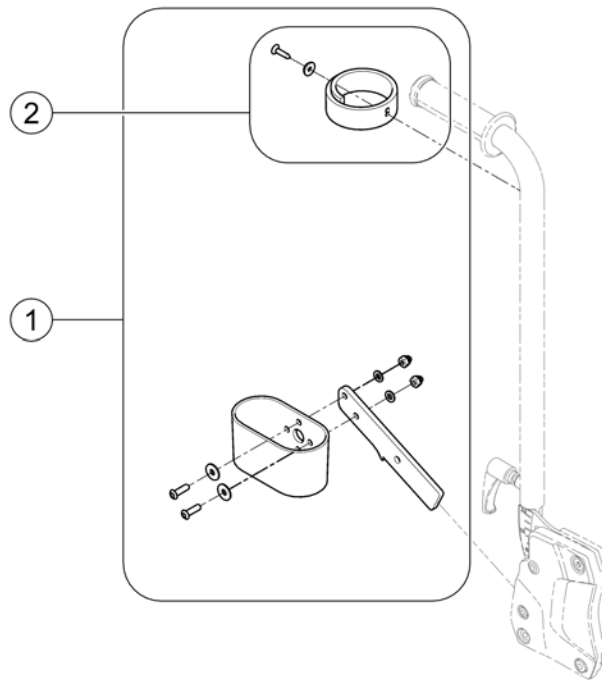

Pos.	Part number	Description	Valid from → until	Qty
1	SP1570443	Cane holder, manual backrest		1
2	SP1420493	Belt for caneholder		1

Cane holder (Modulite: back posts pow recliner)


1571837.jpg

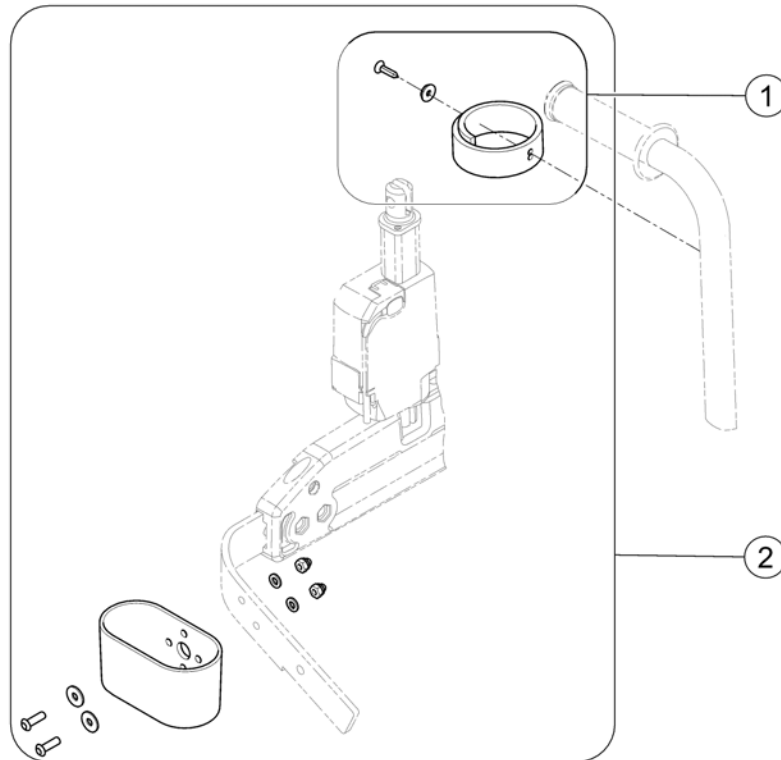

Pos.	Part number	Description	Valid from → until	Qty
1	SP1570382	Cane holder, powered backrest		1
2	SP1420493	Belt for caneholder		1

Fender for Castors


1567879.dwg

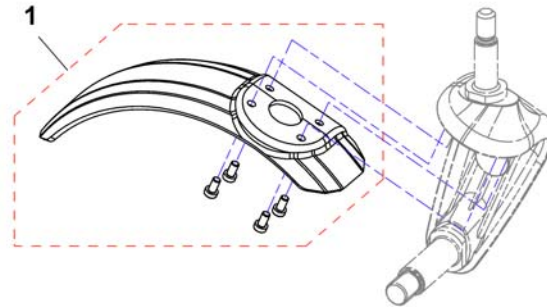

Pos.	Part number	Description	Valid from → until	Qty
1	SP1530530	Fender for Castor, cpl. - RH		1
1	SP1530553	Fender for Castor, cpl. - LH		1

Fenders for Drive Wheels => 04.01.11

	RH		LH			RH		LH			RH		LH	
	1	2	3	4		1	2	3	4		1	2	3	4
Kurze Sitzplatte Short Seatplate					Mittlere Sitzplatte Middle Seatplate					Lange Sitzplatte Long Seatplate				
T=43 B=38	✓	✓	✗	✗	T=48 B=38	✓	✓	✗	✗	T=53 B=38	✓	✓	✗	✗
T=43 B=43	✓	✓	✗	✗	T=48 B=43	✓	✓	✗	✗	T=53 B=43	✓	✓	✗	✗
T=43 B=48	✓	✓	✗	✗	T=48 B=48	✓	✓	✗	✗	T=53 B=48	✗	✗	✓	✓
T=43 B=53	✓	✓	✗	✗	T=48 B=53	✗	✗	✓	✓	T=53 B=53	✗	✗	✓	✓
T=41 B=38	✓	✓	✗	✗	T=46 B=38	✓	✓	✗	✗	T=51 B=38	✓	✓	✗	✗
T=41 B=43	✓	✓	✗	✗	T=46 B=43	✓	✓	✗	✗	T=51 B=43	✓	✓	✗	✗
T=41 B=48	✓	✓	✗	✗	T=46 B=48	✓	✓	✗	✗	T=51 B=48	✗	✗	✓	✓
T=41 B=53	✓	✓	✗	✗	T=46 B=53	✓	✓	✗	✗	T=51 B=53	✗	✗	✓	✓
T=38 B=38	✓	✓	✗	✗	T=43 B=38	✓	✓	✗	✗	T=48 B=38	✓	✓	✗	✗
T=38 B=43	✓	✓	✗	✗	T=43 B=43	✓	✓	✗	✗	T=48 B=43	✓	✓	✗	✗
T=38 B=48	✓	✓	✗	✗	T=43 B=48	✓	✓	✗	✗	T=48 B=48	✗	✗	✓	✓
T=38 B=53	✓	✓	✗	✗	T=43 B=53	✓	✓	✗	✗	T=48 B=53	✗	✗	✓	✓
Recaro Seat	✓	✓	✗	✗	Optimist Seat	✓	✓	✗	✗					

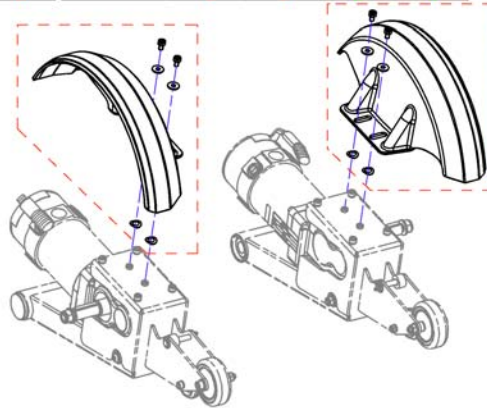


Pos.	Part number	Description	Valid from → until	Qty
1	SP1530557	Mud Guard, long, RH		1
2	SP1530558	Mud Guard, long, LH		1
3	SP1530559	Mud Guard, short, RH		1
4	SP1530560	Mud Guard, short, LH		1

Fenders for Drive Wheels (SSD-Motors)

1534858X.ind


	RH		LH			RH		LH			RH		LH	
	1	2	3	4		1	2	3	4		1	2	3	4
Kurze Sitzplatte Short Seatplate					Mittlere Sitzplatte Middle Seatplate					Lange Sitzplatte Long Seatplate				
T=430 B=380	✓	✓	✗	✗	T=480 B=380	✓	✓	✗	✗	T=530 B=380	✓	✓	✗	✗
T=430 B=430	✓	✓	✗	✗	T=480 B=430	✓	✓	✗	✗	T=530 B=430	✓	✓	✗	✗
T=430 B=480	✓	✓	✗	✗	T=480 B=480	✓	✓	✗	✗	T=530 B=480	✓	✓	✗	✗
T=430 B=530	✓	✓	✗	✗	T=480 B=530	✗	✗	✓	✓	T=530 B=530	✗	✗	✓	✓
T=410 B=380	✓	✓	✗	✗	T=460 B=380	✓	✓	✗	✗	T=510 B=380	✓	✓	✗	✗
T=410 B=430	✓	✓	✗	✗	T=460 B=430	✓	✓	✗	✗	T=510 B=430	✓	✓	✗	✗
T=410 B=480	✓	✓	✗	✗	T=460 B=480	✓	✓	✗	✗	T=510 B=480	✓	✓	✗	✗
T=410 B=530	✓	✓	✗	✗	T=460 B=530	✗	✗	✓	✓	T=510 B=530	✗	✗	✓	✓
T=380 B=380	✓	✓	✗	✗	T=430 B=380	✓	✓	✗	✗	T=480 B=380	✓	✓	✗	✗
T=380 B=430	✓	✓	✗	✗	T=430 B=430	✓	✓	✗	✗	T=480 B=430	✓	✓	✗	✗
T=380 B=480	✓	✓	✗	✗	T=430 B=480	✓	✓	✗	✗	T=480 B=480	✓	✓	✗	✗
T=380 B=530	✓	✓	✗	✗	T=430 B=530	✗	✗	✓	✓	T=480 B=530	✗	✗	✓	✓
Recaro Seat	✓	✓	✗	✗	Optimist Seat	✗	✗	✓	✓					

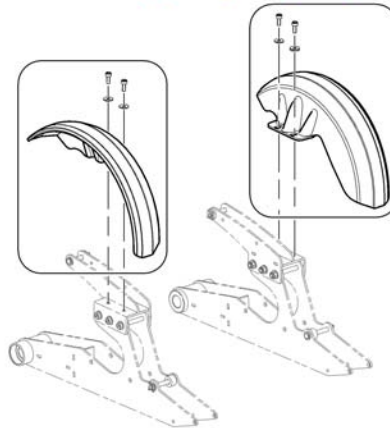

Pos.	Part number	Description	Valid from → until	Qty
1	SP1540339	Mud Guard, long, RH		1
2	SP1540340	Mud Guard, long, LH		1
3	SP1540341	Mud Guard, short, RH		1
4	SP1540342	Mud Guard, short, LH		1

Fenders (GGB Motors)



1583913.iso

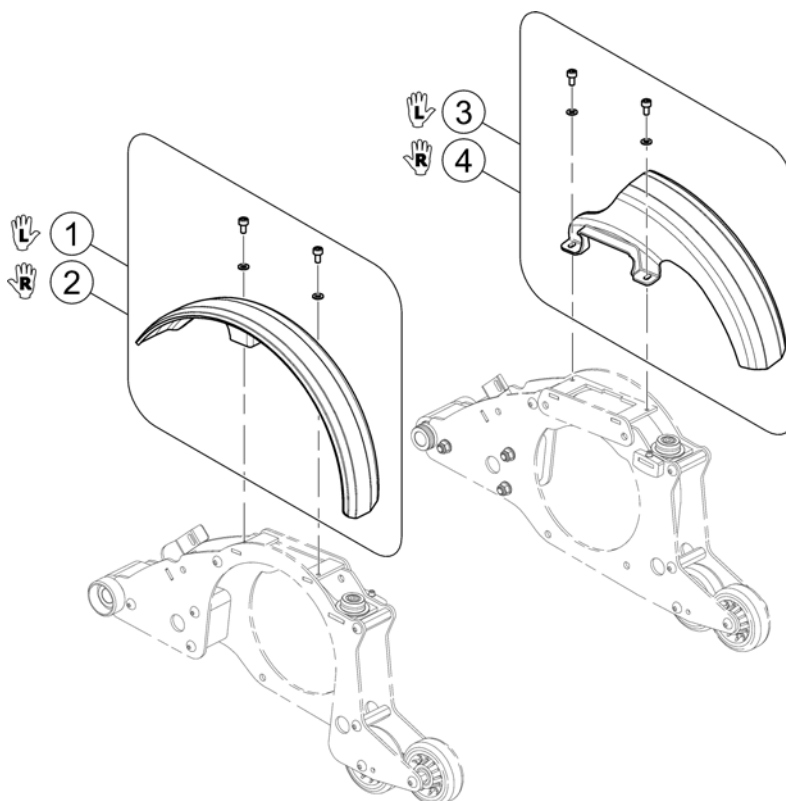

Pos.	Part number	Description	Valid from → until	Qty
1	SP1583917	Fender LH, long [1275/1290/1483]+[1150/1152/1158/1199/2100] / [1275/1290/1483]+[[1180]-[1978/1980/1981]]		1
2	SP1583918	Fender RH, long [1275/1290/1483]+[1150/1152/1158/1199/2100] / [1275/1290/1483]+[[1180]-[1978/1980/1981]]		1
3	SP1583919	Fender LH, short [1275/1290/1483]+[[1180/1171]+[1978/1980/1981]]		1
4	SP1583920	Fender RH, short [1275/1290/1483]+[[1180/1171]+[1978/1980/1981]]		1


Trays

158830 rev

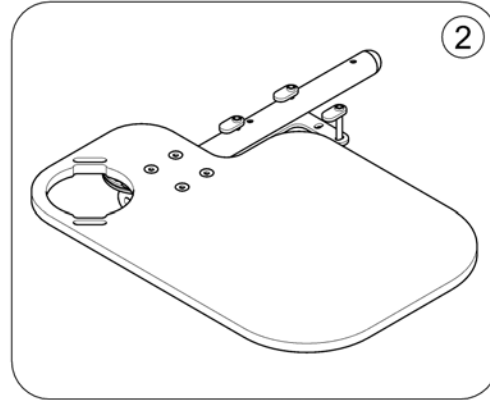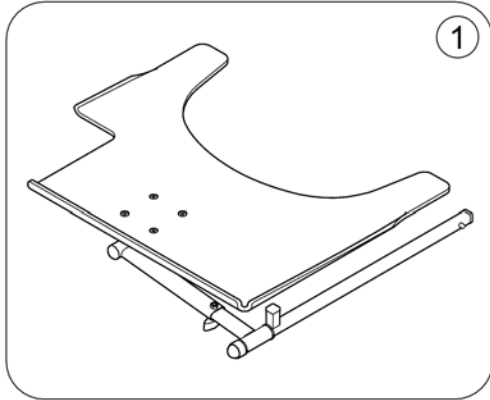

Pos.	Part number	Description	Valid from → until	Qty
1	-	Trays		1
2	-	Trays (Ultra Low Maxx Seat)		1

Trays

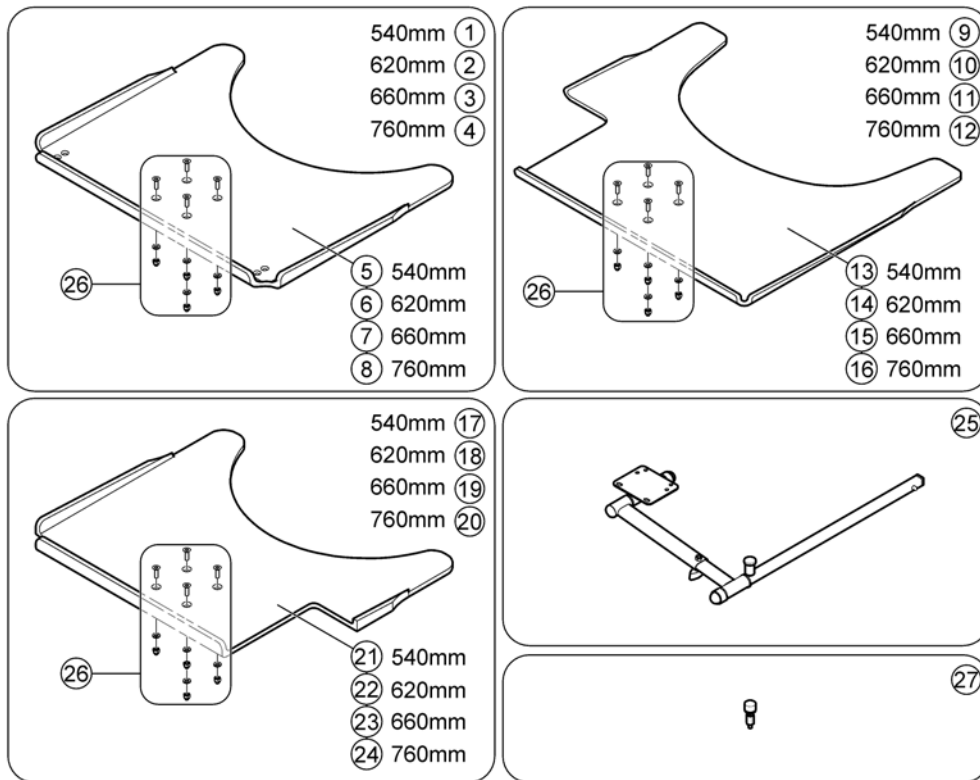


Pos.	Part number	Description	Valid from → until	Qty
1	SP1548198	Swing-Out Tray, 540mm, Seatwidth 330mm to 430mm, no Cutout, cpl. with holder		1
2	SP1548201	Swing-Out Tray, 620mm, Seatwidth 380mm to 480mm, no Cutout, cpl. with holder		1
3	SP1422954	Swing-Out Tray, 660mm, Seatwidth 430mm to 530mm, no Cutout, cpl. with holder		1
4	SP1584644	Swing-Out Tray, 760mm, Seatwidth 480mm to 580mm, no Cutout, cpl. with holder		1
5	SP1535050	Tray table, 540mm, no Cutout		1
6	SP1535010	Tray table, 620mm, no Cutout		1
7	SP1535011	Tray table, 660mm, no Cutout		1
8	SP1586049	Tray table, 760mm, no Cutout		1
9	SP1548138	Swing-Out Tray, 540mm, Seatwidth 330mm to 430mm, Cutout RH, cpl. with holder		1
10	SP1548199	Swing-Out Tray, 620mm, Seatwidth 380mm to 480mm, Cutout RH, cpl. with holder		1
11	SP1422953	Swing-Out Tray, 660mm, Seatwidth 430mm to 530mm, Cutout RH, cpl. with holder		1
12	SP1584663	Swing-Out Tray, 760mm, Seatwidth 480mm to 580mm, Cutout RH, cpl. with holder		1
13	SP1535051	Tray table, 540mm, Cutout RH		1
14	SP1535012	Tray table, 620mm, Cutout RH		1
15	SP1535049	Tray table, 660mm, Cutout RH		1
16	SP1586048	Tray table, 760mm, Cutout RH		1
17	SP1548197	Swing-Out Tray, 540mm, Seatwidth 330mm to 430mm, Cutout LH, cpl. with holder		1
18	SP1548200	Swing-Out Tray, 620mm, Seatwidth 380mm to 480mm, Cutout LH, cpl. with holder		1
19	SP1422952	Swing-Out Tray, 660mm, Seatwidth 430mm to 530mm, Cutout LH, cpl. with holder		1
20	SP1584662	Swing-Out Tray, 760mm, Seatwidth 480mm to 580mm, Cutout LH, cpl. with holder		1
21	SP1535009	Tray table, 540mm, Cutout LH		1
22	SP1534974	Tray table, 620mm, Cutout LH		1

Trays


Pos.	Part number	Description	Valid from → until	Qty
23	SP1534975	Tray table, 660mm, Cutout LH		1
24	SP1586047	Tray table, 760mm, Cutout LH		1
25	SPZ177011	Tray Support Fixing Kit cpl.SW 380/505 mm		1
26	Z177025	Fastening-Kit Tray		1
27	SP1590090	Tray Support bumper for Soft Armpad (increased height)		1

Trays (Ultra Low Maxx Seat)

1588507 Rev

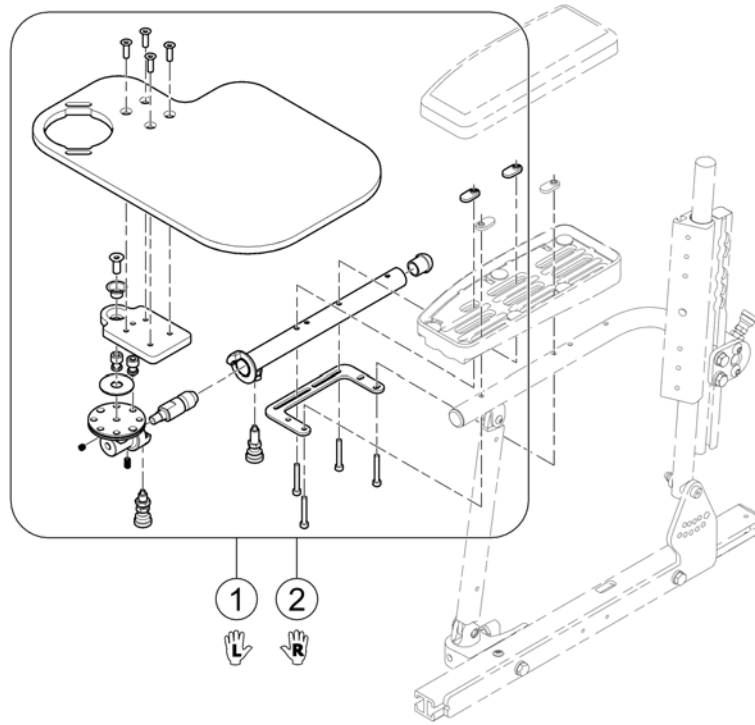


Pos.	Part number	Description	Valid from → until	Qty
1	SPTRC2065	Flip and fold away Tray LH		1
2	SPTRC2066	Flip and fold away Tray RH		1

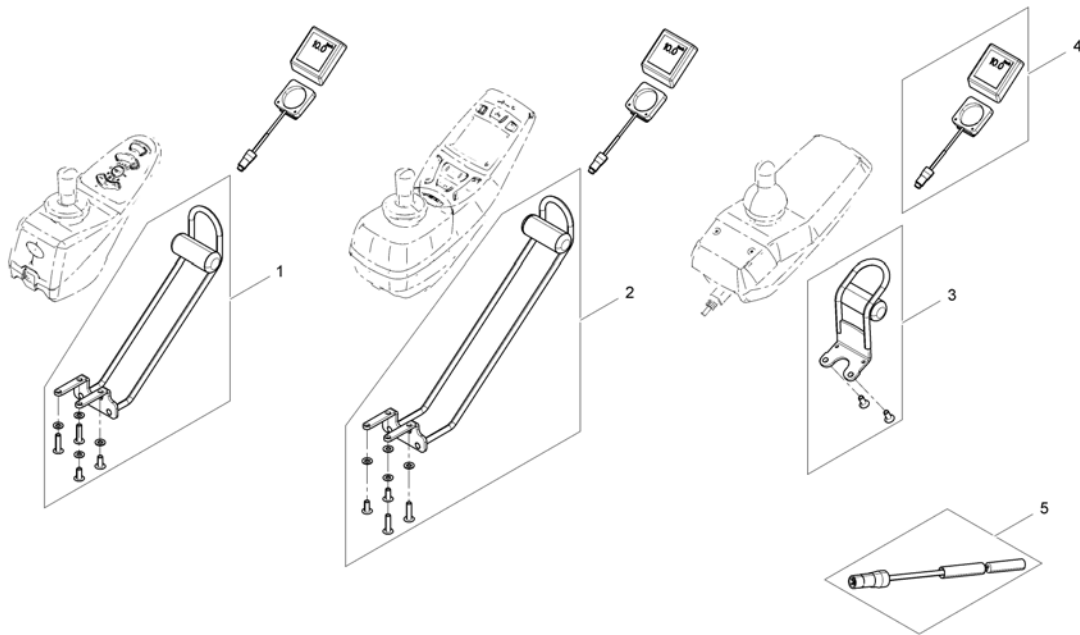

Pos.	Part number	Description	Valid from → until	Qty
1	SP1530563	Speedometer holder REM A & REM B		1
2	SP1530564	Speedometer holder REM 550		1
3	SP1546263	Speedometer holder REM24		1
4	Z183259	Speedometer		1
5	1421733	Reed-sensor cable l=2500mm		1

Mounting-kit km-Counter


1540353X.iso

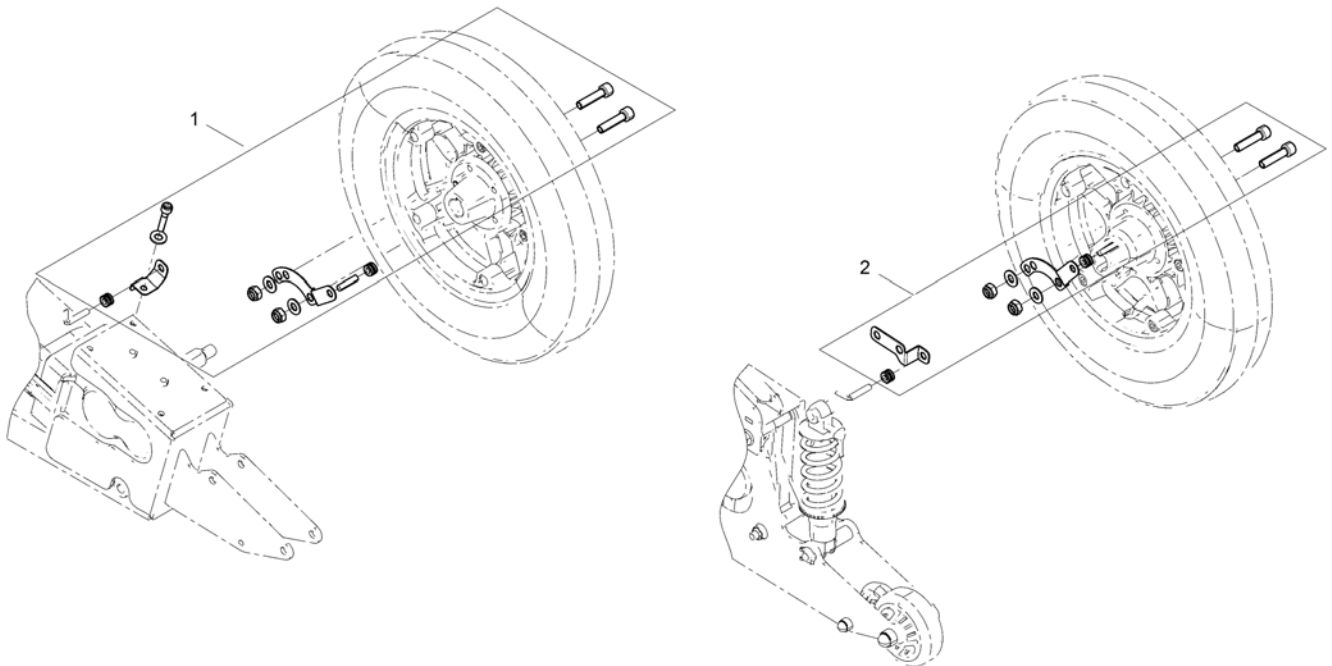

Pos.	Part number	Description	Valid from → until	Qty
1	SP1548705	Mounting-kit Speedometer sensor		1
2	SP1540353	Mounting-kit Speedometer sensor SSD-motors		1

Hour Counter


1567259.iso

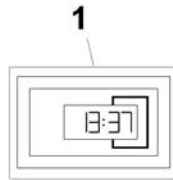

Pos.	Part number	Description	Valid from → until	Qty
-	-	Not for retrofitting - only spares!		1
1	SP1530275	Hour counter		1
1	SP1608033	Hour counter (GGB Motors)		1

Bag


1567256 iso

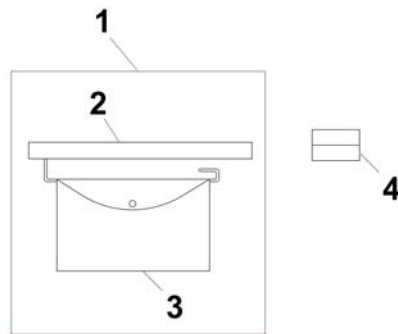

Pos.	Part number	Description	Valid from → until	Qty
1	SP1430417	Bag for side panel, cpl. LH		1
2	SP1430855	Holder for armrest bag		1
3	SP5314967	Bag for armrest		1
4	F370032-9005	Holder for side part with mirror/brake RAL 9		1

Toolkit


1567260 no

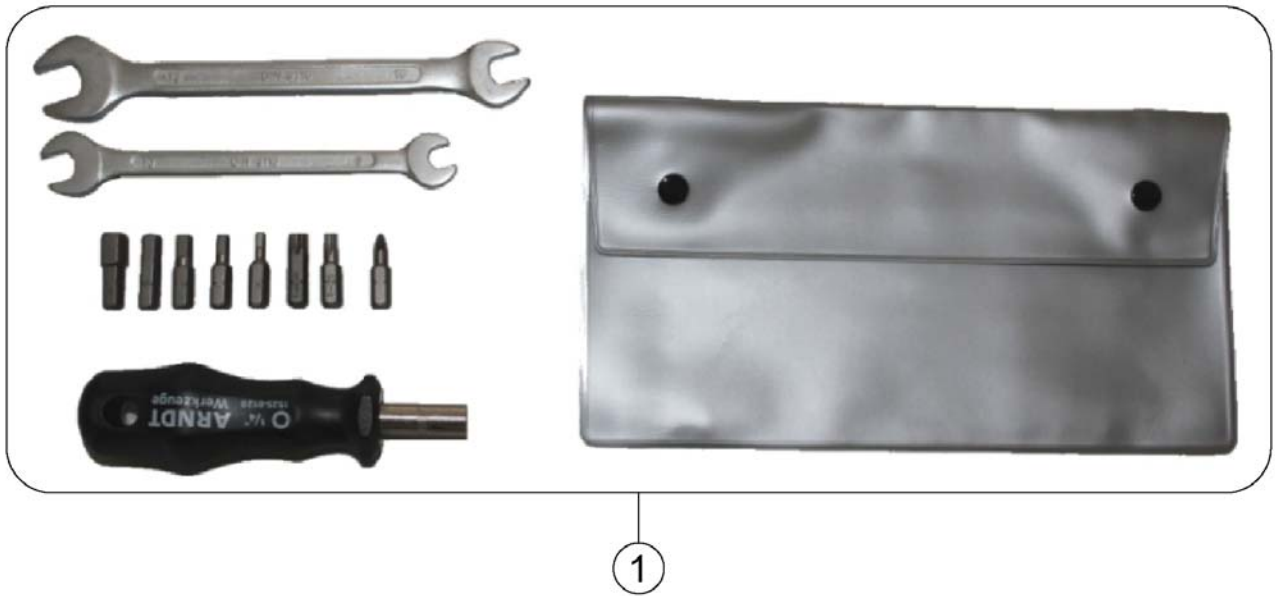

Pos.	Part number	Description	Valid from → until	Qty
1	SP1422868	Toolkit with bag		1

T-Nuts for Accessories Modulite & ULM Seat

1574067 Rev

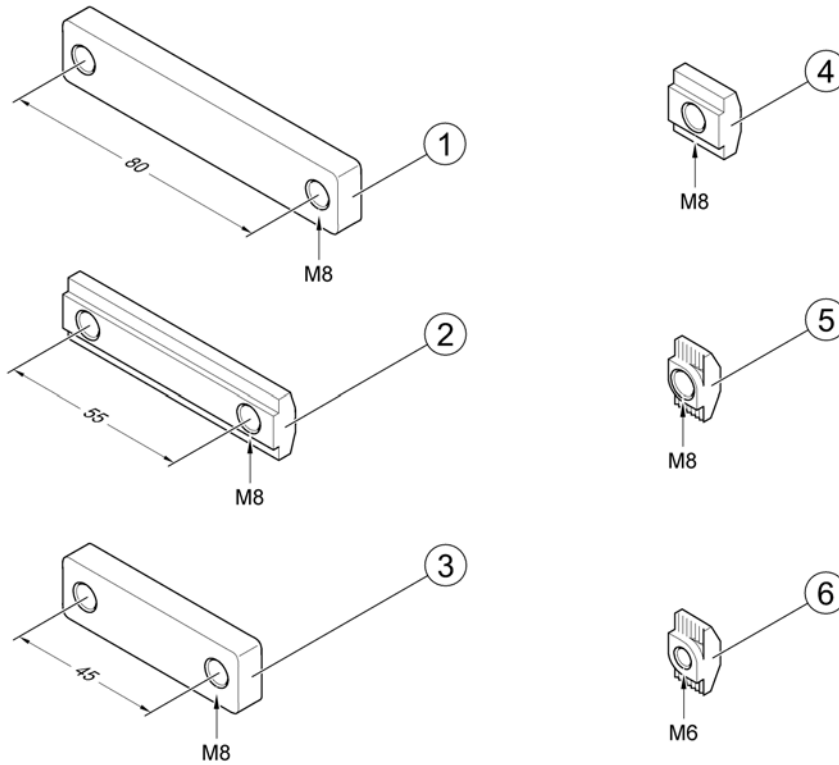

Pos.	Part number	Description	Valid from → until	Qty
1	SP1556401	T-Nut, hole distance 80mm, 2x M8		1
2	SP1572444	T-Nut, hole distance 55mm, 2x M8		1
3	SP1555168	T-Nut, hole distance 45mm, 2x M8		1
4	SP1557341	T-Nut, 1x M8		1
5	SP1573158	T-Nut, 1x M8		1
6	SP1573721	T-Nut, 1x M6		1



Yes, you can.®



Belgium & Luxemburg:

Invacare nv, Autobaan 22, B-8210 Loppem
Tel: (32) (0)50 83 10 10, Fax: (32) (0)50 83 10 11
belgium@invacare.com / www.invacare.be

Denmark:

Invacare A/S, Sdr. Ringvej 37, DK-2605 Brøndby
Tel: (45) (0)36 90 00 00, Fax: (45) (0)36 90 00 01
denmark@invacare.com / www.invacare.dk

Deutschland:

Invacare Aquatec GmbH, Alemannenstraße 10, D-88316 Isny
Tel: (49) (0)75 62 7 00 0, Fax: (49) (0)75 62 7 00 66
info@invacare-aquatec.com / www.invacare-aquatec.de

European Distributor Organisation:

Invacare, Kleiststraße 49, D-32457 Porta Westfalica
Tel: (49) (0)57 31 754 540, Fax: (49) (0)57 31 754 541
eda@invacare.com / www.invacare.eu.com

España:

Invacare SA, c/Areny s/n, Polígon Industrial de Celrà, E-17460 Celrà (Girona)
Tel: (34) (0)972 49 32 00, Fax: (34) (0)972 49 32 20
contactsp@invacare.com / www.invacare.es

France:

Invacare Poirier SAS, Route de St Roch, F-37230 Fondettes
Tel: (33) (0)2 47 62 64 66, Fax: (33) (0)2 47 42 12 24
contactfr@invacare.com / www.invacare.fr

Ireland:

Invacare Ireland Ltd, Unit 5 Seatown Business Campus, Seatown Road, Swords, County Dublin - Ireland
Tel: (353) 1 810 7084, Fax: (353) 1 810 7085
ireland@invacare.com / www.invacare.ie

Italia:

Invacare Mecc San s.r.l., Via dei Pini 62, I-36016 Thiene (VI)
Tel: (39) 0445 38 00 59, Fax: (39) 0445 38 00 34
italia@invacare.com / www.invacare.it

Nederland:

Invacare BV, Celsiusstraat 46, NL-6716 BZ Ede
Tel: (31) (0)318 695 757, Fax: (31) (0)318 695 758
nederland@invacare.com, csede@invacare.com / www.invacare.nl

Norge:

Invacare AS, Grensesvingen 9, Postboks 6230, Etterstad, N-0603 Oslo
Tel: (47) (0)22 57 95 00, Fax: (47) (0)22 57 95 01
norway@invacare.com, island@invacare.com / www.invacare.no

Österreich:

Invacare Austria GmbH, Herzog Odilostrasse 101, A-5310 Mondsee
Tel: (43) 6232 5535 0, Fax: (43) 6232 5535 4
info-austria@invacare.com / www.invacare.at

Portugal:

Invacare Lda, Rua Estrada Velha, 949, P-4465-784 Leça do Balio
Tel: (351) (0)225 1059 46147, Fax: (351) (0)225 1057 39
portugal@invacare.com / www.invacare.pt

Sverige & Suomi:

Invacare AB, Fagerstagatan 9, S-163 91 Spånåsa
Tel: (46) (0)8 761 70 90, Fax: (46) (0)8 761 81 08
sweden@invacare.com, finland@invacare.com / www.invacare.se

Switzerland:

Invacare AG, Benkenstrasse 260, CH-4108 Witterswil
Tel: (41) (0)61 487 70 80, Fax: (41) (0)61 487 70 81
switzerland@invacare.com / www.invacare.ch

United Kingdom:

Invacare Limited, Pencoed Technology Park, Pencoed, Bridgend CF35 5AQ
Switchboard Tel: (44) (0)1656 776200, Fax: (44) (0)1656 776201
Customer services Tel: (44) (0)1656 776222, Fax: (44) (0)1656 776220
uk@invacare.com / www.invacare.co.uk