

Spare Parts Catalogue

Version: 2016/04/20a EN



Invacare® FDX



Yes, you can.®

General Information

For any order of spare parts the following information is required:

- Product type*
- Product name & model*
- Serial Number*
- Part-Number*
- Part-Description*

Exploded view drawings and code numbers are not contractual and subject to modification, in relation to Invacare® continuous product improvement policy to meet end users needs.

The reproduction or use of this work, or of any substantial part of this work, in any form or by any electronic, mechanical or other means, including photocopying, scanning and recording, and in any information storage and retrieval system, including electronic transmission, is forbidden without the prior written permission of Invacare® International Sarl.



Yes, you can.®

TABLE OF CONTENTS

Information & User Manuals

User Manuals FDX	9
User Manuals REM 24 SD	10
User Manuals REM A & B	11
User Manuals REM 550	12
User Manuals G90A	13
User Manuals G91S	14
User Manuals REM 34 & REM 41 Tray/Centre Control	15
User Manuals Chin Control (2015-10-01 =>)	16
User Manuals Compact Dual Control	17
User Manuals IDC (Intuitive Dual Control)	18
User Manuals ACS Handprogrammer	19

Chassis

Covers	20
Tie-Down	21
Anti-Tipper	22
Low Rider Option	23

Seat

Seat	24
Standard Seat	25
Seatsupport, manuell	26
Seattilt 30°	27
Lifter	28
Bellow	29
Seat Plate	30
Seat Cushion	31
Easy Adapt Seat	32
Seatsupport, manuell	26
Seattilt 30°	27
Lifter	28
Bellow	29
Seat Frame	33
Seat Plate	30
Adapter Kit for Universal Seat	34
Seat Cushion	35
Recaro Seat	37
Seatsupport, manuell	26
Seattilt 30°	27
Lifter	28
Bellow	29
Seat Frame & Plate for Recaro Seat	38
Recaro Seat	39
Change list Recaro 24V	40
Modulite Seat	41
Tilt 0°, 4°, 8°	42
Tilt 30°	43
Lifter	44
One Piece Seat Plate	45



Yes, you can.®

TABLE OF CONTENTS

Telescopic Seat Frame and Plate	46
Telescopic Seat Frame for Tilt 0°, 4°, 8°	47
Telescopic Seat Frame for Tilt 30° / Lifter	48
Armrest bracket	49
Legrest Support	50
Seat Plate	51
Telescopic Seat Frame and Sling Seat	52
Telescopic Seat Frame for Tilt 0°, 4°, 8°	53
Telescopic Seat Frame for Tilt 30° / Lifter	54
Armrest bracket	49
Legrest Support	55
Sling Seat	56
Telescopic Seat Frame and Universal Adapter	57
Telescopic Seat Frame for Tilt 12° / 20°	58
Telescopic Seat Frame for Tilt 30° / Lifter	48
Armrest bracket	49
Legrest Support	50
Adapter Kit for Universal Seat	59
Seat Cushions "Standard", "Matrx Flo-Tech" & "Matrx Vi"	60
Seat Cushions "Matrx Libra"	63

Backrest

Backrest	64
Backrest for Standard Seat	65
Standard Backrest	66
Backrest for EasyAdapt Seat	67
Backrest adjustment	68
Following armrests	69
Backrestcover for EasyAdapt	70
Backplate EasyAdapt	71
Backplate EasyAdapt for DAHL Docking Station	72
Backrest Cushion	73
Push Handle EasyAdapt	75
Tension adjustable Sling Back	76
"Universal"-Backrest frame	77
Backrest for Modulite Seat	78
Support for Backrest Frame	79
Backrest Frame	80
Sling Back, tension adjustable	81
Backrest support	82
Backposts	83
Connection tube	84
Back Recliner / Stabilization bracket	85
Backrest Cushions	86
Flex 3 / Laguna	87
Matrx Posture Back	89
Matrx Elite T/R Back	90
Matrx Elite Deep Back	91
Matrx Elite (Full) Back	92
Flex3 back, tension adjustable	93



Yes, you can.®

TABLE OF CONTENTS

Legrests / Hangers

Legrest Support, Vari series 07/2005 ==>	94
Hanger Vari F	95
Hanger Vari A (04/2009 =>)	96
Hangers ADM LH/RH manual	97
Hanger Vari ADE/ADP electric (07/2005 =>)	98
Calfpad Holder (07/2005 =>)	99
Central legrests - Std-/ Easy Adapt Seat	100
Support central legrests - Std-/ Easy Adapt Seat	101
Central Legrest man.	102
Centr. Legrest pow. LNX	103
Central legrests - Modulite Seat	104
Support Centr. Legrest man. - Modulite Seat	105
Support Centr. Legrest LNX pow. - Modulite Seat	106
Central Legrest man.	102
Central Legrest pow. LNX	107

Footplates / Support tubes

Footrests Vari series	108
Footrests Vari, Flip-Up (01.08.10 =>)	109
Footrests Vari, Flip-Up & angle adjustable (01.08.10 =>)	110

Armrests

Armrests	111
Armrest for Standard Seat	112
Fixed Armrest, removable	113
Armrests for EasyAdapt Seat	114
Fixed Armrest, removable	113
Following armrests	69
Skirtguard for following armrest	115
Armrest for Recaro Seat	116
Fixed Armrest, removable	113
Armrests for Modulite Seat	117
T-Armrest	118
Padded Armrest	119
Inrigged Armrest	120
Flip-Up Armrest	121
Following Armrest	122
Hip Support	123

Castors

Rearwheels	124
------------	-----

Drive Wheels

Drive wheels 12.5"	125
Drive wheels 14"	126

Propulsion



Yes, you can.®

TABLE OF CONTENTS

Overview Propulsion SSD	127
Swingarms	130
Springs	131

Electronic

Electronic	132
ACS Electronic Modules	133
Powermodule & Gyroskop	134
Powermodules	135
Upgrade-Kit 80A ACS => ACS2, Dragon / Tiger / Topan	136
Upgrade-Kit 80A ACS => ACS2, Dragon Vertic	137
Retrofit-Kit 80A ACS => ACS2	138
Upgrade-Kit 80A ACS => ACS2, Storm® Vertic	139
Holder ACS2 Powermodule	140
ACT for ACS 2	141
Holder ACT	142
Lightboard	143
ACS Remotes & Brackets	144
REM 24 SD with brackets	145
Remote REM 24SD	146
Remote Bracket Standard	147
Remote Bracket Swing away & height adjust.	148
Remote Bracket Attendant for Standard-/EasyAdapt-Seat	149
Remote Bracket Attendant for Recaro-Seat	150
REM A & B with brackets	151
Remote Bracket Standard	147
Remote Bracket Swing away & height adjust.	152
Remote Bracket Attendant for Standard-/EasyAdapt-Seat	149
Remote Bracket Attendant for Recaro-Seat	150
REM 550 with brackets	153
Remote Bracket Standard	147
Remote Bracket Swing away & height adjust.	148
Remote Bracket Attendant for Standard-/EasyAdapt-Seat	149
Remote Bracket Attendant for Recaro-Seat	150
REM 34 with brackets	154
Remote Bracket Standard	155
Remote Bracket Swing away & height adjust.	156
G90A with brackets	157
Remote Bracket Standard	158
Remote Bracket Swing away & height adjust.	159
Remote Bracket Attendant for Standard-/EasyAdapt-Seat	160
Remote Bracket Attendant for Recaro-Seat	161
G91S with brackets	162
Remote Bracket Standard	163
Centre control REM 41	164
Small Chin Control (old) with brackets	165
Small Chincontrol	166
Framework for chincontrol	167
Chin Control (new) with mounting interface - Modulite Seat	168
Chin Control DX-RJM	169
Chin Control powered - Details	170
Chin Control manual - Details	171



Yes, you can.®

TABLE OF CONTENTS

Mounting Hardware - Chin Control	172
Dual Control with brackets	173
Remote Bracket Dual Control for Standard-/EasyAdapt-Seat	174
Remote Bracket Dual Control for Modulite Seat	175
Remote Bracket Dual Control for Recaro Seat	176
Compact Dual Control with brackets	177
Compact Dual Control	178
Bracket, Compact Dual Control - short	179
Bracket, Compact Dual Control - long	180
Intuitive Dual Control (IDC) with bracket	181
IDC with bracket	182
Intuitive Dual Control (IDC)	183
User Manuals IDC (Intuitive Dual Control)	18
Height adjustable bracket IDC	184
ACS Cables & Accessories	185
Bus-Cables ACS	186
T-Adapter ACS	187
T-Adapter - Modulite Seat	188
Programming Kits	189
Magnetkey for Remotes	190
ACS Components	191
iPortal Modules and Accessories	192
Remote, Joystick, Control	193
Joystick Knob	194
Buddy Button	195
Piko Button	196
Interface, Adapter, Other	197

Cable, Charger & Batteries

Batteries	198
Chargers	199
Main harnesses	200

Safety Options & Accessories

Light System (German homologation)	201
Rear Light Protector Kit	202
Frontlight for Modulite (German homologation)	203
LED-Frontlight	204
LED-Frontlight for Modulite	205
LED-Rearlight	206
Rear reflectors	207
Reflective Sign	208
Adapter Kit for DAHL Docking Station	209
Backplate EasyAdapt for DAHL Docking Station	72
Lap Belts / H-Belt	210
Lap Belts (Modulite Seat)	211
H-Belts (Modulite Seat)	212
Rearview Mirror	213

Seat Accessories

Headrests	214
-----------------	-----



Yes, you can.®

TABLE OF CONTENTS

Headrest with cheek support	215
Headrest & Bracket	216
Headrestholder EasyAdapt Seat	217
Headrest, sidewise adjustable	218
Lateral Supports	219
Lateral Support - Modulite Seat	220
Lateral Support (Dartex)	221
Lateral Support, extended arm (Dartex)	222
Lateral Support - EasyAdapt Seat	223
Lateral Support (Dartex)	224
Lateral Support, multi adjustable with flexible pad	225
Bag	226

Accessories

Caneholder	227
Caneholder for Standard Seat (with fixed seat frame)	228
Cane holder (Easy Adapt Seat & Std-Seat)	229
Cane holder (Recaro Seat)	230
Cane holder (Modulite: backrest frame)	231
Cane holder (Modulite: back posts man recliner)	232
Cane holder (Modulite: back posts pow recliner)	233
Trays	234
km-Counter & Bracket	236
Mounting-kit km-Counter	237
Hour Counter	238
KLICKfix® bag	239
KLICKfix® bracket for Fixed Armrest	240
KLICKfix® bracket for T-Armrest	241
KLICKfix® bracket for Inrigged/Flip-Up/Following Armrest (Modulite)	242
Bag	243
Toolkit	244
T-Nuts for Accessories Modulite Seat	245

User Manuals FDX

languages.iso

BG Български	FR Français	NL Nederlands	SV Svenska
CS Čeština	HE עברית	NO Norsk	UK Українська
DA Dansk	HR Hrvatski	PL Polski	
DE Deutsch	HU Magyar	PT Português	
EN English	IT Italiano	RU Русский	
ES Español	JA 日本語	RO Română	
ET Eesti	LV Latviešu	SK Slovenčina	
FI Suomi	LT Lietuvių	SL Slovenščina	

Pos.	Part number	Description	Valid from → until	Qty
DA	SP1540315	User Manual FDX Danish		1
DE	SP1541360	User Manual FDX German		1
EN	SP1541361	User Manual FDX English		1
ES	SP1540310	User Manual FDX Spanish		1
FI	SP1540313	User Manual FDX Finnish		1
FR	SP1540308	User Manual FDX France		1
IT	SP1540309	User Manual FDX Italiano		1
LT	SP1602722	User Manual FDX Lithuanian		1
NL	SP1540311	User Manual FDX Dutch		1
PT	SP1545716	User Manual FDX Portuguese		1
RU	SP1549024	User Manual FDX Russian		1
SV	SP1540312	User Manual FDX Swedish		1

User Manuals REM 24 SD

languages.iso

BG Български	FR Français	NL Nederlands	SV Svenska
CS Čeština	HE עברית	NO Norsk	UK Українська
DA Dansk	HR Hrvatski	PL Polski	
DE Deutsch	HU Magyar	PT Português	
EN English	IT Italiano	RU Русский	
ES Español	JA 日本語	RO Română	
ET Eesti	LV Latviešu	SK Slovenčina	
FI Suomi	LT Lietuvių	SL Slovenščina	

Pos.	Part number	Description	Valid from → until	Qty
CS	SP1533252	User Manual REM24SD Czech		1
DA	1436146	User Manual REM24SD Danish		1
DE	1436142	User Manual REM24SD German		1
EN	1436144	User Manual REM24SD English		1
ES	1436149	User Manual REM24SD Spanish		1
ET	SP1533253	User Manual REM24SD Estonian		1
FI	1436386	User Manual REM24SD Finnish		1
FR	1436143	User Manual REM24SD French		1
HR	SP1533251	User Manual REM24SD Croatian		1
HU	SP1533254	User Manual REM24SD Hungarian		1
IT	1436385	User Manual REM24SD Italian		1
LV	SP1543152	User Manual REM24SD Latvian		1
LT	SP1533255	User Manual REM24SD Lithuanian		1
NL	1436148	User Manual REM24SD Dutch		1
NO	1436147	User Manual REM24SD Norwegian		1
PL	SP1533256	User Manual REM24SD Polish		1
PT	1436150	User Manual REM24SD Portuguese		1
SL	SP1533257	User Manual REM24SD Slovenian		1
SV	1436145	User Manual REM24SD Swedish		1

User Manuals REM A & B

languages.iso

BG Български	FR Français	NL Nederlands	SV Svenska
CS Čeština	HE עברית	NO Norsk	UK Українська
DA Dansk	HR Hrvatski	PL Polski	
DE Deutsch	HU Magyar	PT Português	
EN English	IT Italiano	RU Русский	
ES Español	JA 日本語	RO Română	
ET Eesti	LV Latviešu	SK Slovenčina	
FI Suomi	LT Lietuvių	SL Slovenščina	

Pos.	Part number	Description	Valid from → until	Qty
DA	1529171	User Manual REM A & B - Danish		1
DE	1529172	User Manual REM A & B - German		1
EN	1529173	User Manual REM A & B - English		1
ES	1529174	User Manual REM A & B - Spanish		1
FI	1529228	User Manual REM A & B - Finnish		1
FR	1529229	User Manual REM A & B - French		1
IT	1529230	User Manual REM A & B - Italian		1
NL	1529231	User Manual REM A & B - Dutch		1
NO	1529232	User Manual REM A & B - Norwegian		1
PT	1529233	User Manual REM A & B - Portuguese		1
SV	1529234	User Manual REM A & B Swedish		1
HR	SP1533258	User Manual REM A & B - Croatian		1
CS	SP1533259	User Manual REM A & B - Czech		1
ET	SP1533265	User Manual REM A & B - Estonian		1
HU	SP1533266	User Manual REM A & B - Hungarian		1
LT	SP1533267	User Manual REM A & B - Lithuanian		1
PL	SP1533268	User Manual REM A & B - Polish		1
SL	SP1533269	User Manual REM A & B - Slovenian		1
RO	SP1533270	User Manual REM A & B - Romanian		1

User Manuals REM 550

languages.iso

BGБългарски	FRFrançais	NLNederlands	SVSvenska
CSČeština	HEעברית	NONorsk	UKУкраїнська
DADansk	HRHrvatski	PLPolski	
DEDeutsch	HUMagyar	PTPortuguês	
ENEnglish	ITItaliano	RURусский	
ESEspañol	JAJapanese	RORomână	
ETEesti	LVLatviešu	SKSlovenčina	
FISuomi	LTLietuvių	SLSlovenščina	

Pos.	Part number	Description	Valid from → until	Qty
DA	1528088	User Manual REM550 Danish		1
DE	1528089	User Manual REM550 German		1
EN	1528090	User Manual REM550 English		1
ES	1528091	User Manual REM550 Spanish		1
FI	1528092	User Manual REM550 Finsk		1
FR	1528093	User Manual REM550 French		1
IT	1528094	User Manual REM550 Italiano		1
NL	1528095	User Manual REM550 Dutch		1
NO	1528096	User Manual REM550 Norsk		1
PT	1528097	User Manual REM550 Portuguese		1
SV	1528098	User Manual REM550 Swedish		1
HR	SP1533279	User Manual REM550 Croatian		1
CS	SP1533280	User Manual REM551 Czech		1
ET	SP1533281	User Manual REM552 Estonian		1
HU	SP1533282	User Manual REM553 Hungarian		1
LT	SP1533283	User Manual REM554 Lithuanian		1
PL	SP1533284	User Manual REM555 Polish		1
SL	SP1533285	User Manual REM556 Slovenian		1

User Manuals G90A

languages.iso

BG	Български	FR	Français	NL	Nederlands	SV	Svenska
CS	Čeština	HE	עברית	NO	Norsk	UK	Українська
DA	Dansk	HR	Hrvatski	PL	Polski		
DE	Deutsch	HU	Magyar	PT	Português		
EN	English	IT	Italiano	RU	Русский		
ES	Español	JA	日本語	RO	Română		
ET	Eesti	LV	Latviešu	SK	Slovenčina		
FI	Suomi	LT	Lietuvių	SL	Slovenščina		

Pos.	Part number	Description	Valid from → until	Qty
EN	1523674	User Manual G90 English		1

User Manuals G91S

languages.iso

BGБългарски	FRFrançais	NLNederlands	SVSvenska
CSČeština	HEעברית	NONorsk	UKУкраїнська
DADansk	HRHrvatski	PLPolski	
DEDeutsch	HUMagyar	PTPortuguês	
ENEnglish	ITItaliano	RURусский	
ESEspañol	JAJapanese	RORomână	
ETEesti	LVLatviešu	SKSlovenčina	
FISuomi	LTLietuvių	SLSlovenščina	

Pos.	Part number	Description	Valid from → until	Qty
	DE 1536441	User Manual G91 German		1
	DA 1537929	User Manual G91 Danish		1
	NL 1537930	User Manual G91 Dutch		1
	EN 1537931	User Manual G91 English		1
	FI 1537932	User Manual G91 Finnish		1
	FR 1537933	User Manual G91 French		1
	IT 1537934	User Manual G91 Italian		1
	NO 1537935	User Manual G91 Norwegian		1
	PT 1537936	User Manual G91 Portuguese		1
	ES 1537937	User Manual G91 Spanish		1
	SV 1537938	User Manual G91 Swedish		1

User Manuals REM 34 & REM 41 Tray/Centre Control

BG	Български	FR	Français	NL	Nederlands	SV	Svenska
CS	Čeština	HE	עברית	NO	Norsk	UK	Українська
DA	Dansk	HR	Hrvatski	PL	Polski		
DE	Deutsch	HU	Magyar	PT	Português		
EN	English	IT	Italiano	RU	Русский		
ES	Español	JA	日本語	RO	Română		
ET	Eesti	LV	Latviešu	SK	Slovenčina		
FI	Suomi	LT	Lietuvių	SL	Slovenščina		

languages.iso

Pos.	Part number	Description	Valid from → until	Qty
DA	SP1538022	User Manual REM41 Tray Control Danish		1
EN	SP1538023	User Manual REM41 Tray Control English		1
ES	SP1538024	User Manual REM41 Tray Control Spanish		1
FR	SP1538025	User Manual REM41 Tray Control French		1
IT	SP1538026	User Manual REM41 Tray Control Italian		1
NL	SP1538027	User Manual REM41 Tray Control Dutch		1
NO	SP1538028	User Manual REM41 Tray Control Norwegian		1
PT	SP1538029	User Manual REM41 Tray Control Portuguese		1
SV	SP1538030	User Manual REM41 Tray Control Swedish		1
CS	SP1538031	User Manual REM41 Tray Control Czech		1
SL	SP1538032	User Manual REM41 Tray Control Slovenian		1
DE	SP1541402	User Manual REM41 Tray Control German		1

User Manuals Chin Control (2015-10-01 =>)

languages.iso

BG Български	FR Français	NL Nederlands	SV Svenska
CS Čeština	HE עברית	NO Norsk	UK Українська
DA Dansk	HR Hrvatski	PL Polski	
DE Deutsch	HU Magyar	PT Português	
EN English	IT Italiano	RU Русский	
ES Español	JA 日本語	RO Română	
ET Eesti	LV Latviešu	SK Slovenčina	
FI Suomi	LT Lietuvių	SL Slovenščina	

Pos.	Part number	Description	Valid from → until	Qty
CS	SP1589326	User Manual Chin Control - Czech		1
DA	SP1589394	User Manual Chin Control - Danish		1
DE	SP1589395	User Manual Chin Control - German		1
EN	SP1589325	User Manual Chin Control - English		1
ES	SP1589535	User Manual Chin Control - Spanish		1
FI	SP1589436	User Manual Chin Control - Finnish		1
FR	SP1589437	User Manual Chin Control - French		1
HR	SP1589438	User Manual Chin Control - Croatian		1
IT	SP1589439	User Manual Chin Control - Italian		1
LT	SP1589440	User Manual Chin Control - Lithuanian		1
NL	SP1589441	User Manual Chin Control - Dutch		1
NO	SP1589442	User Manual Chin Control - Norwegian		1
PL	SP1589443	User Manual Chin Control - Polish		1
PT	SP1589444	User Manual Chin Control - Portuguese		1
SL	SP1589445	User Manual Chin Control - Slovenian		1
SV	SP1589465	User Manual Chin Control - Swedish		1

User Manuals Compact Dual Control

languages.iso

BG Български	FR Français	NL Nederlands	SV Svenska
CS Čeština	HE עברית	NO Norsk	UK Українська
DA Dansk	HR Hrvatski	PL Polski	
DE Deutsch	HU Magyar	PT Português	
EN English	IT Italiano	RU Русский	
ES Español	JA 日本語	RO Română	
ET Eesti	LV Latviešu	SK Slovenčina	
FI Suomi	LT Lietuvių	SL Slovenščina	

Pos.	Part number	Description	Valid from → until	Qty
BG	SP1589186	User Manual Compact Dual Control - Bulgarian		1
CS	SP1589313	User Manual Compact Dual Control - Czech		1
DA	SP1588488	User Manual Compact Dual Control - Danish		1
DE	SP1588489	User Manual Compact Dual Control - German		1
EN	SP1588462	User Manual Compact Dual Control - English		1
ES	SP1588496	User Manual Compact Dual Control - Spanish		1
ET	SP1589311	User Manual Compact Dual Control - Estonian		1
FI	SP1588490	User Manual Compact Dual Control - Finnish		1
FR	SP1588491	User Manual Compact Dual Control - French		1
HR	SP1589314	User Manual Compact Dual Control - Croatian		1
HU	SP1589315	User Manual Compact Dual Control - Hungarian		1
IT	SP1588492	User Manual Compact Dual Control - Italian		1
LV	SP1589317	User Manual Compact Dual Control - Latvian		1
LT	SP1589316	User Manual Compact Dual Control - Lithuanian		1
NL	SP1588493	User Manual Compact Dual Control - Dutch		1
NO	SP1588494	User Manual Compact Dual Control - Norwegian		1
PL	SP1589318	User Manual Compact Dual Control - Polish		1
PT	SP1588495	User Manual Compact Dual Control - Portuguese		1
RU	SP1589320	User Manual Compact Dual Control - Russian		1
RO	SP1589319	User Manual Compact Dual Control - Romanian		1
SK	SP1589321	User Manual Compact Dual Control - Slovak		1
SL	SP1589322	User Manual Compact Dual Control - Slovenian		1
SV	SP1588497	User Manual Compact Dual Control - Swedish		1
UK	SP1589323	User Manual Compact Dual Control - Ukrainian		1

User Manuals IDC (Intuitive Dual Control)

languages.iso

BG Български	FR Français	NL Nederlands	SV Svenska
CS Čeština	HE עברית	NO Norsk	UK Українська
DA Dansk	HR Hrvatski	PL Polski	
DE Deutsch	HU Magyar	PT Português	
EN English	IT Italiano	RU Русский	
ES Español	JA 日本語	RO Română	
ET Eesti	LV Latviešu	SK Slovenčina	
FI Suomi	LT Lietuvių	SL Slovenščina	

Pos.	Part number	Description	Valid from → until	Qty
DE	1521480	User Manual IDC - German		1
EN	1521481	User Manual IDC - English		1
FR	1521482	User Manual IDC - French		1
NL	1521483	User Manual IDC - Dutch		1
NO	1521484	User Manual IDC - Norwegian		1
SV	1521485	User Manual IDC - Swedish		1
DA	1521486	User Manual IDC - Danish		1
IT	1522252	User Manual IDC - Italian		1
PT	1522254	User Manual IDC - Portuguese		1
ES	1522255	User Manual IDC - Spanish		1

User Manuals ACS Handprogrammer


languages.iso

BG	Български	FR	Français	NL	Nederlands	SV	Svenska
CS	Čeština	HE	עברית	NO	Norsk	UK	Українська
DA	Dansk	HR	Hrvatski	PL	Polski		
DE	Deutsch	HU	Magyar	PT	Português		
EN	English	IT	Italiano	RU	Русский		
ES	Español	JA	日本語	RO	Română		
ET	Eesti	LV	Latviešu	SK	Slovenčina		
FI	Suomi	LT	Lietuvių	SL	Slovenščina		

Pos.	Part number	Description	Valid from → until	Qty
DE	1419316	User Manual Handheld Programmer German		1
EN	1419317	User Manual Handheld Programmer English		1

Covers


1537236X.dwg

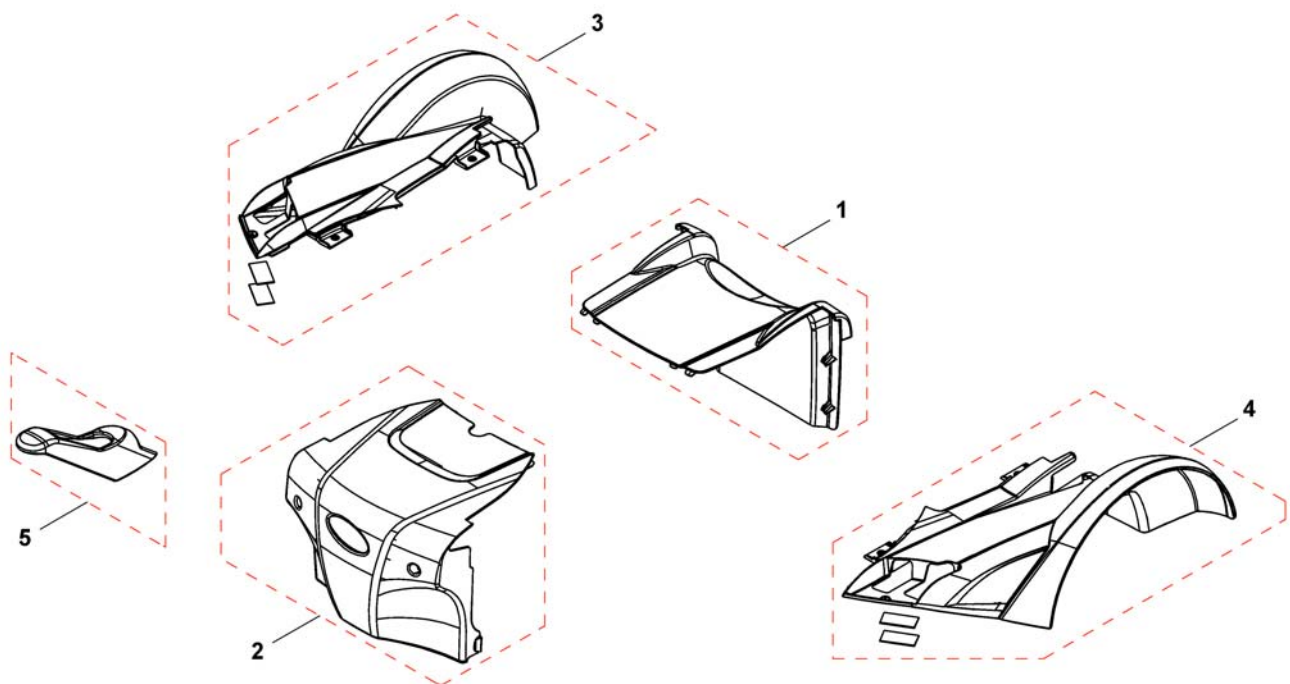

Pos.	Part number	Description	Valid from → until	Qty
-	SP1541766	Cover Kit silver, incl. Mounting Hardware		1
-	SP1541767	Cover Kit red, incl. Mounting Hardware		1
-	1168901	Screw kit for covers		1
1	SP1167990	Frontcover, silver		1
1	SP1167991	Frontcover, red		1
2	SP1541728	Rearcover, silver, cpl.		1
2	SP1541729	Rearcover, red, cpl.		1
3	SP1541730	Cover LH, silver, cpl.		1
3	SP1541731	Cover LH, red, cpl.		1
4	SP1541732	Cover RH, silver, cpl.		1
4	SP1541733	Cover RH, red, cpl.		1
5	SP1173501	Cover suspensionarm RH/LH, silver, cpl.		2
5	SP1541735	Cover suspensionarm RH/LH, red, cpl.		2

Tie-Down


1188553X.dwg

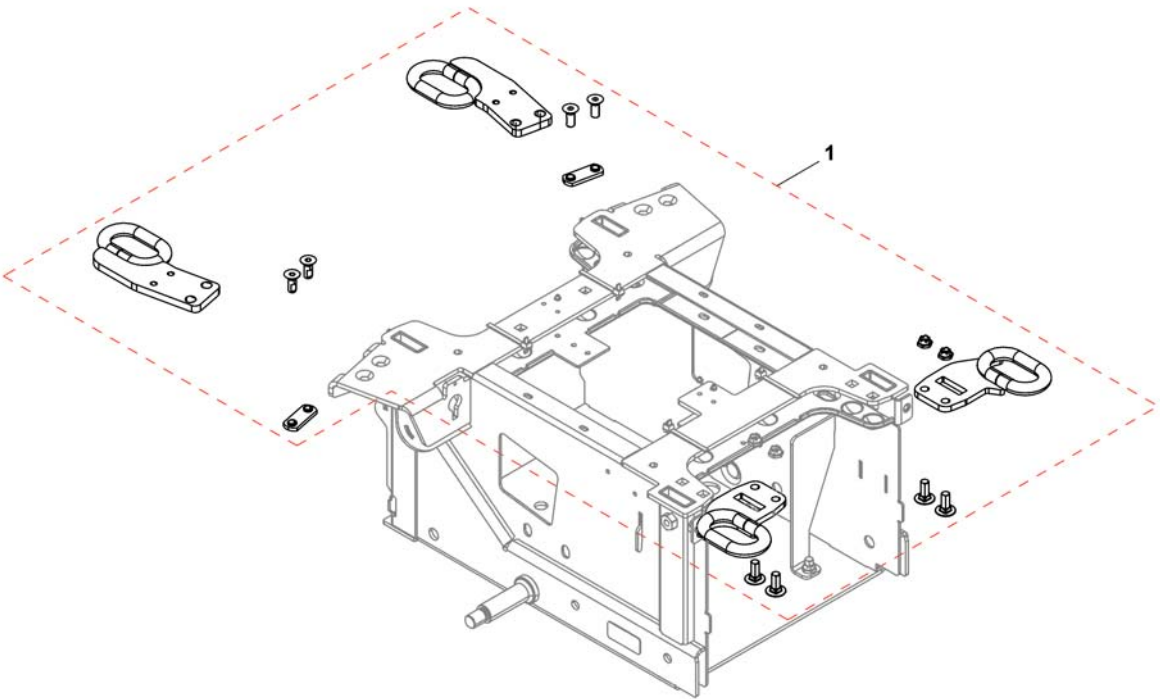


Pos.	Part number	Description	Valid from → until	Qty
1	SP1541803	Tie-Down-Kit		1

Anti-Tipper

1165554.dwg

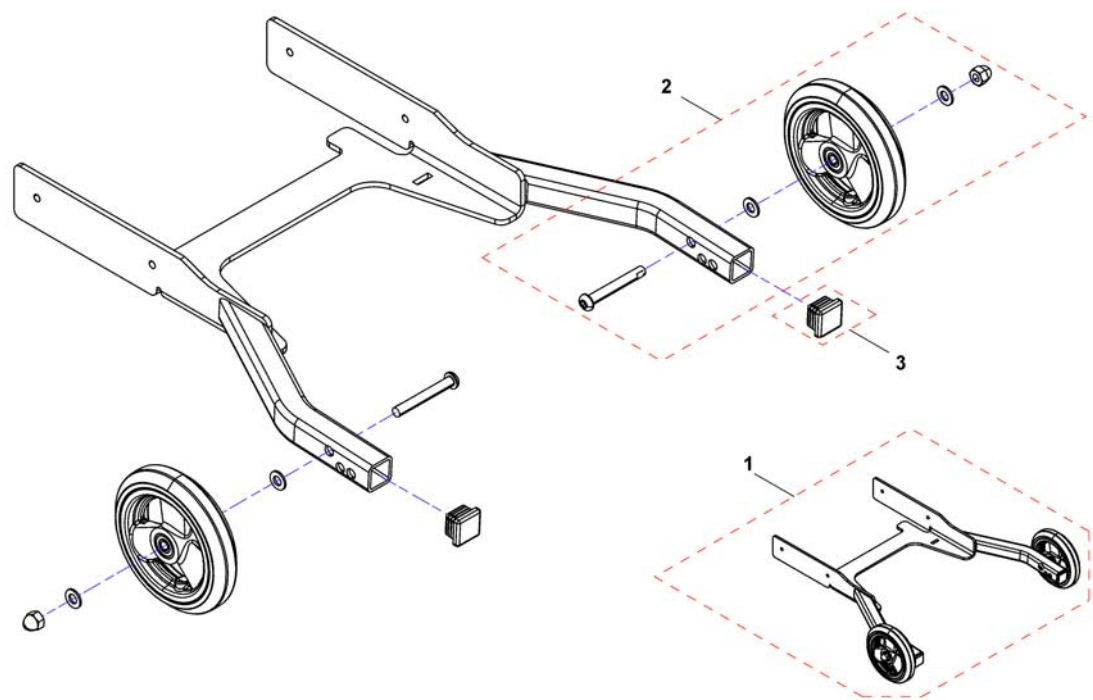

Pos.	Part number	Description	Valid from → until	Qty
1	SP1166194	Anti-Tipper frame		1
2	SP1541802	Anti-Tipp Wheel		1
3	SP1162559	Plug		1

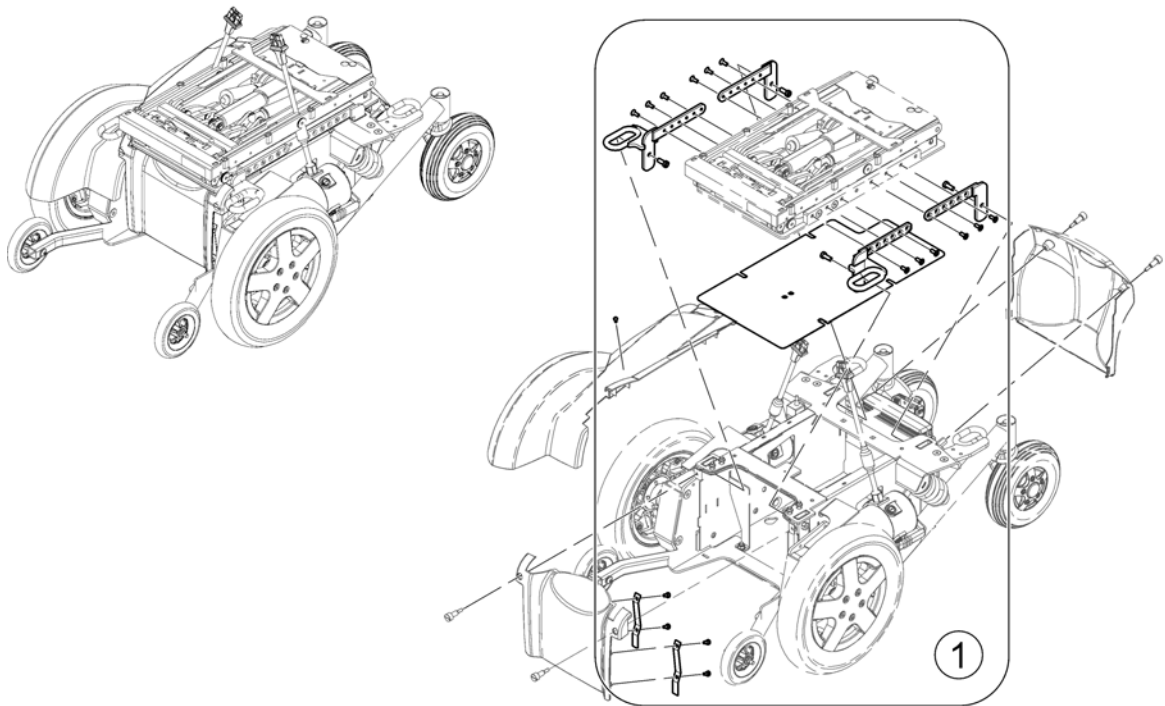


Pos.	Part number	Description	Valid from → until	Qty
1	SP1580180	Low Rider Kit		1

Seat

SFCFDX20 iso




Pos.	Part number	Description	Valid from → until	Qty
1	-	Standard Seat		1
2	-	Easy Adapt Seat		1
3	-	Recaro Seat		1
4	-	Modulite Seat		1

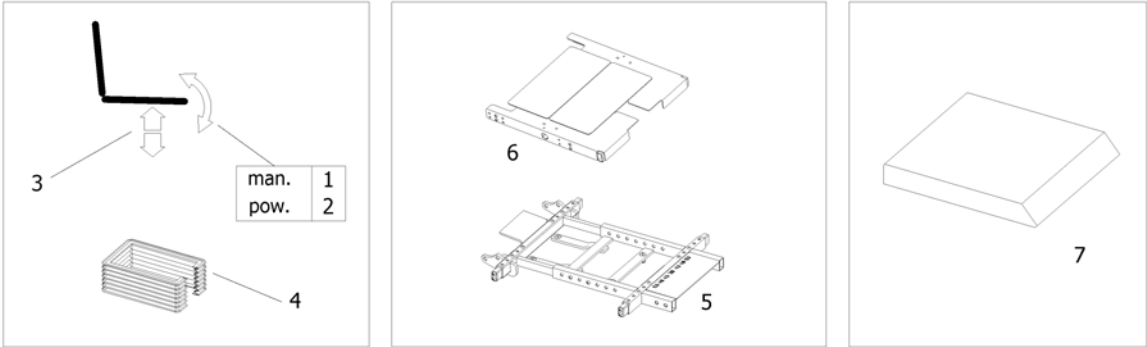

Pos.	Part number	Description	Valid from → until	Qty
1	-	Seatsupport, manuell [1023/1025]		1
2	-	Seattilt 30° [1014]		1
3	-	Lifter [1030]		1
4	-	Bellow [1311]		1
6	-	Seat Plate		1
7	-	Seat Cushion		1


Seatsupport, manuell

1537293X.iso

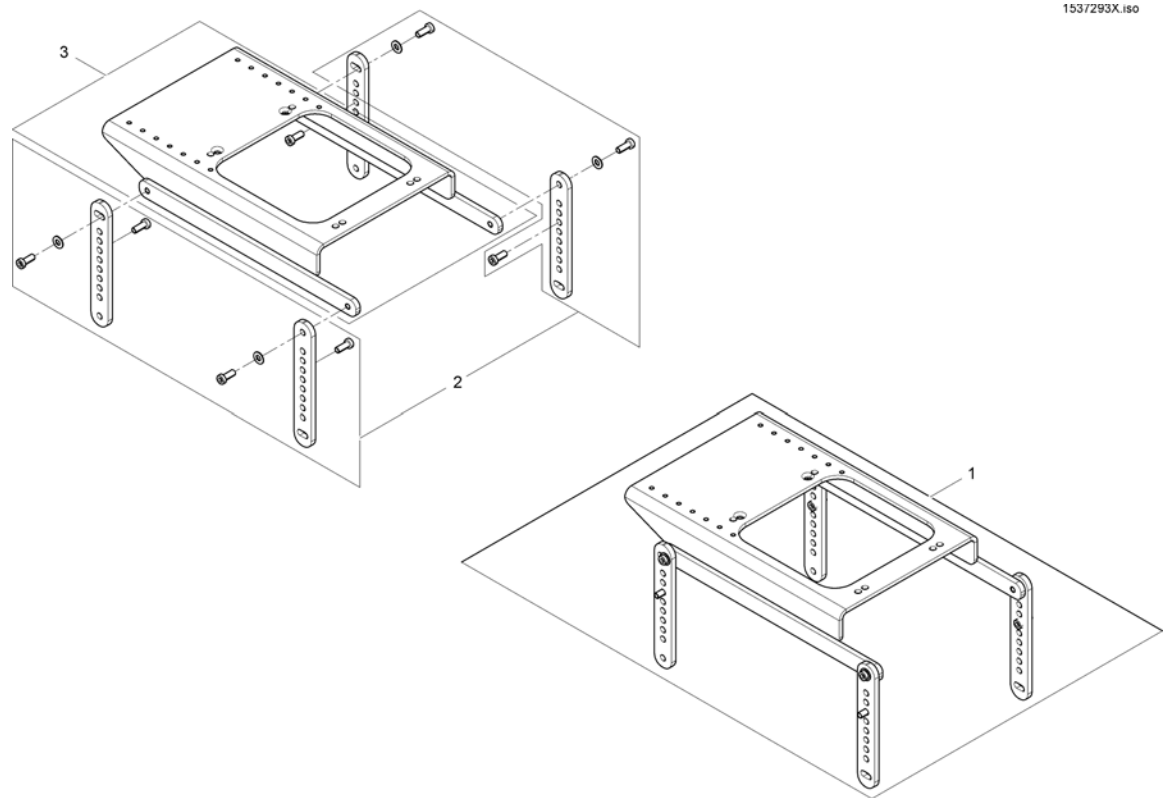

Pos.	Part number	Description	Valid from → until	Qty
1	SP1537239	Seatsupport, manuell		1
2	SP1541736	Seatpost		1
3	SP1536515	Seatadapter		1


1537240X.dwg

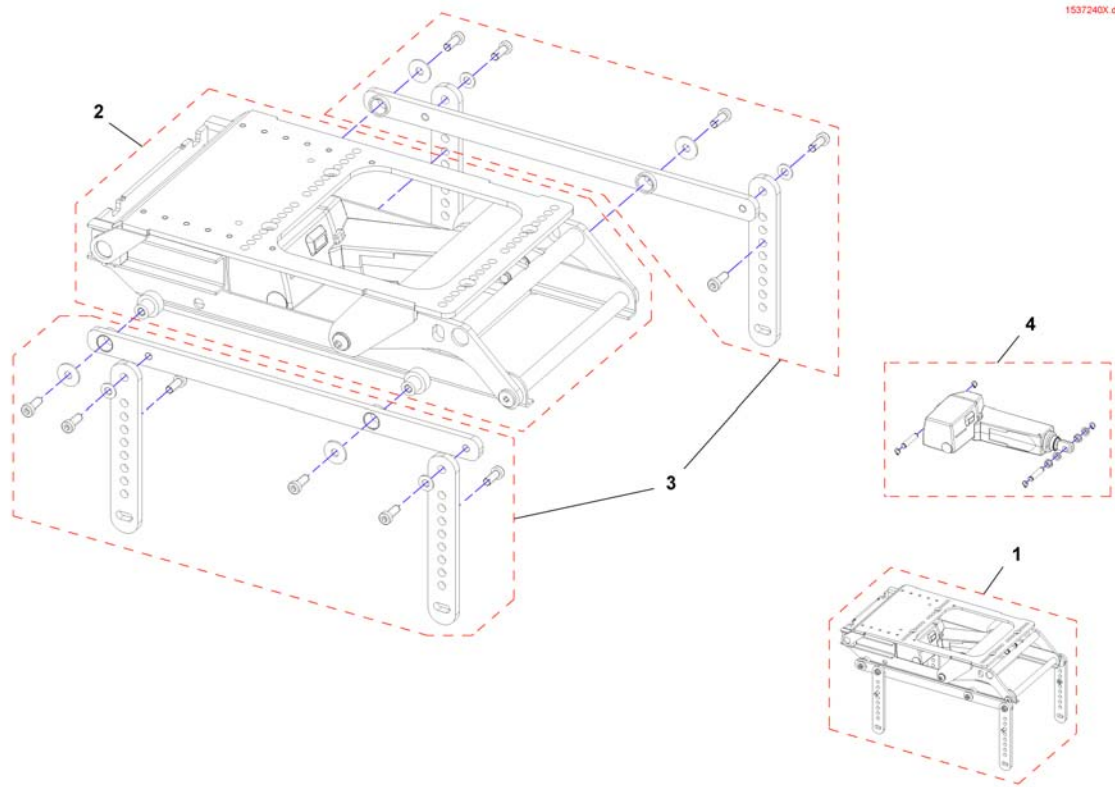


Pos.	Part number	Description	Valid from → until	Qty
1	SP1537240	Seatsupport with 30° seattilt		1
2	SP1533063	Tilt unit 0-28°		1
3	SP1541737	Adapter bracket, cpl.		1
4	SP1544888	Actuator		1

Lifter

1540292X.iso

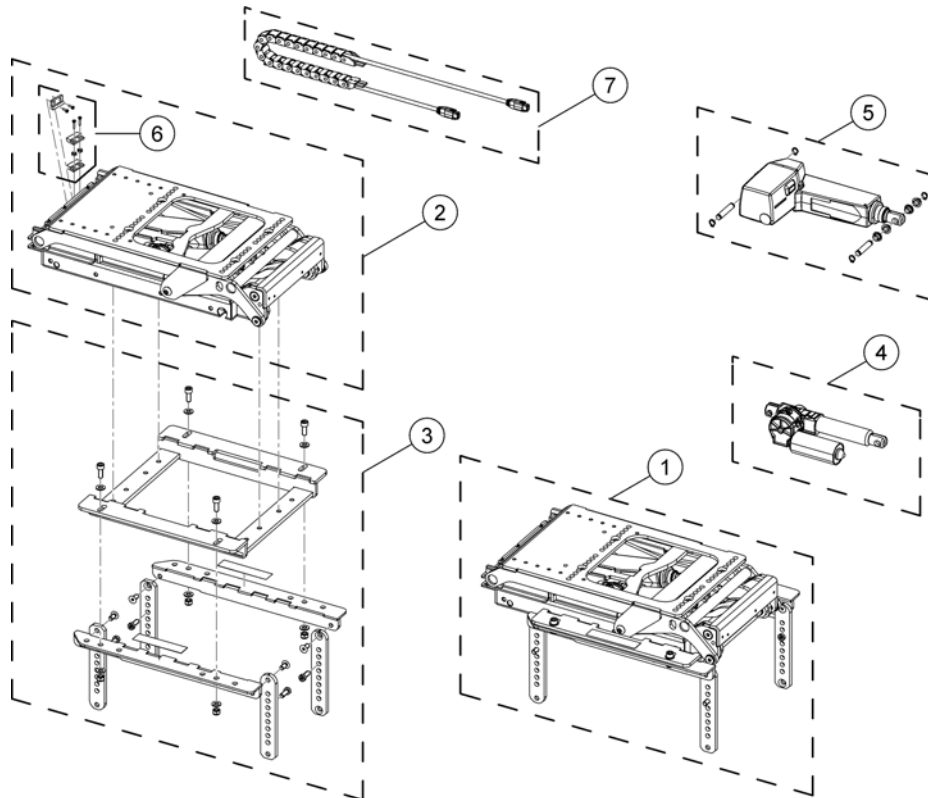


Pos.	Part number	Description	Valid from → until	Qty
-	SP1533562	Actuator cable ACS2 (~1570mm)		1
-	SP1571139	Tilt actuator cable ACS2 (~920mm)		1
1	SP1540292	Seat support incl. lifter		1
2	SP1533423	Lifter		1
3	SP1541738	Seatsupport		1
4	SP1533557	Tilt actuator, ACS2		1
5	SP1533616	Lifter actuator		1
6	SP1536184	Speed reduction sensor		1
7	SP1542965	Energychain with cable		1

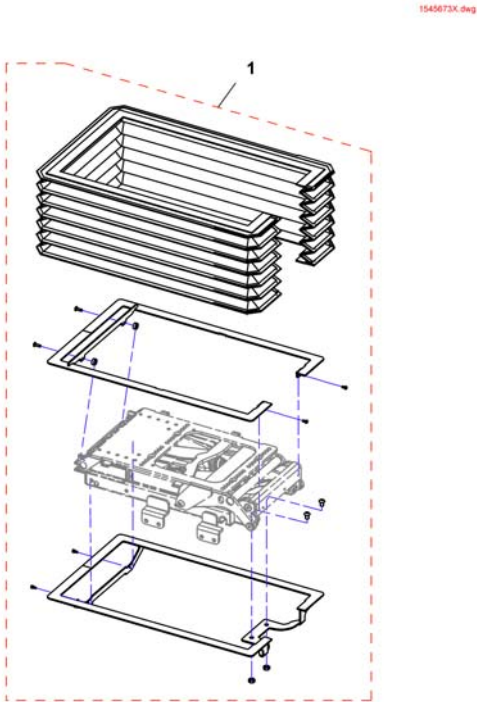

Pos.	Part number	Description	Valid from → until	Qty
	SP1545673	Bellow		1


Seat Plate

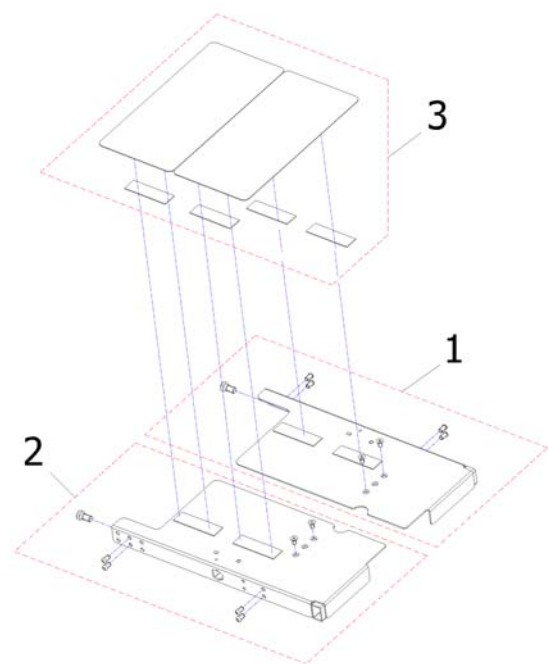

Pos.	Part number	Description	Valid from → until	Qty
1	SP1530486	Seat Plate, short (440mm), LH		1
2	SP1530487	Seat Plate, short (440mm), RH		1
1	SP1530488	Seat Plate, middle (490mm), LH		1
2	SP1530489	Seat Plate, middle (490mm), RH		1
1	SP1530490	Seat Plate, lang (540mm), LH		1
2	SP1530491	Seat Plate, long (540mm), RH		1
3	SP1530492	Cover Plates		1

Seat Cushion


1573211.000

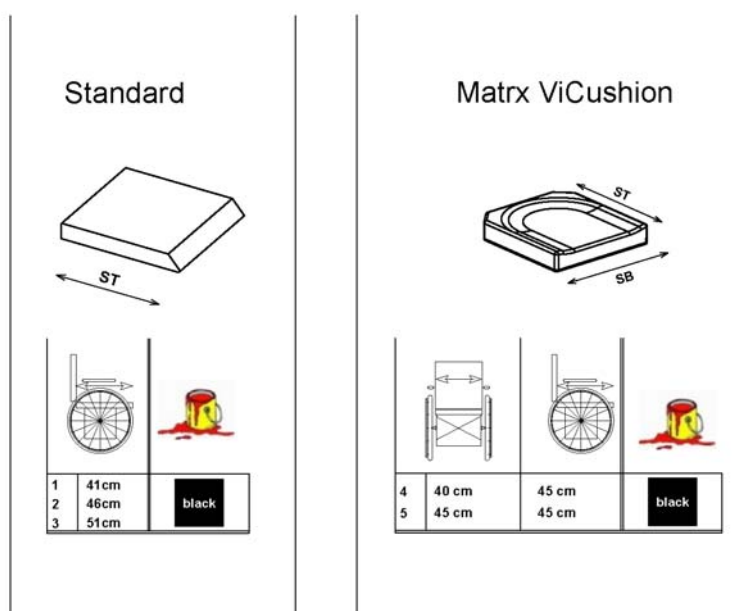

Pos.	Part number	Description	Valid from → until	Qty
1	1420460	Seat Cushion Standard - SW 430mm, SD 410mm [1336]		1
2	1420461	Seat Cushion Standard - SW 430mm, SD 460mm [1336]		1
3	1420462	Seat Cushion Standard - SW 430mm, SD 510mm [1336]		1
4	SP1531760	Seat Cushion Matrix Vi SW 40 x SD 45cm (16" x 18") [2009]		1
5	SP1531761	Seat Cushion Matrix Vi SW 45 x SD 45cm (18" x 18") [2011]		1


Easy Adapt Seat

1574065 iso

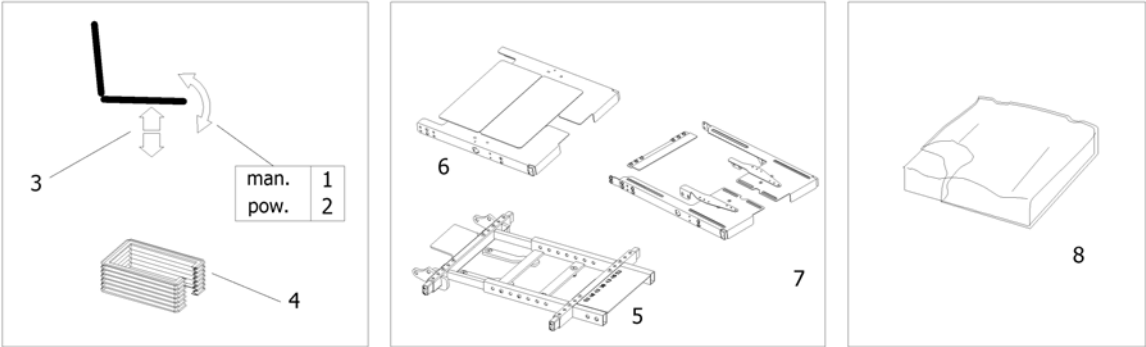

Pos.	Part number	Description	Valid from → until	Qty
1	-	Seatsupport, manuell [1023/1025]		1
2	-	Seattilt 30° [1014]		1
3	-	Lifter [1030]		1
4	-	Bellow [1311]		1
5	-	Seat Frame		1
6	-	Seat Plate		1
7	-	Adapter Kit for Universal Seat [1064]		1
8	-	Seat Cushion		1


1574106 iso

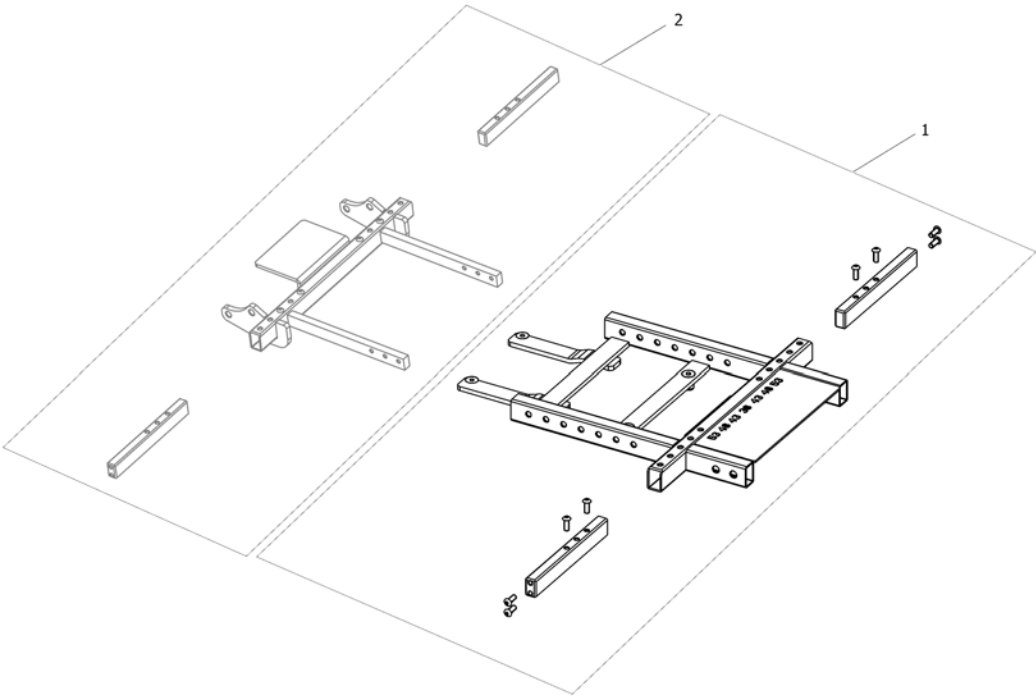

Pos.	Part number	Description	Valid from → until	Qty
1	SP1537242	Seat Frame front		1
2	SP1542967	Rear Seat Frame for EasyAdapt /Standard - Seat		1

Adapter Kit for Universal Seat

1563173.00

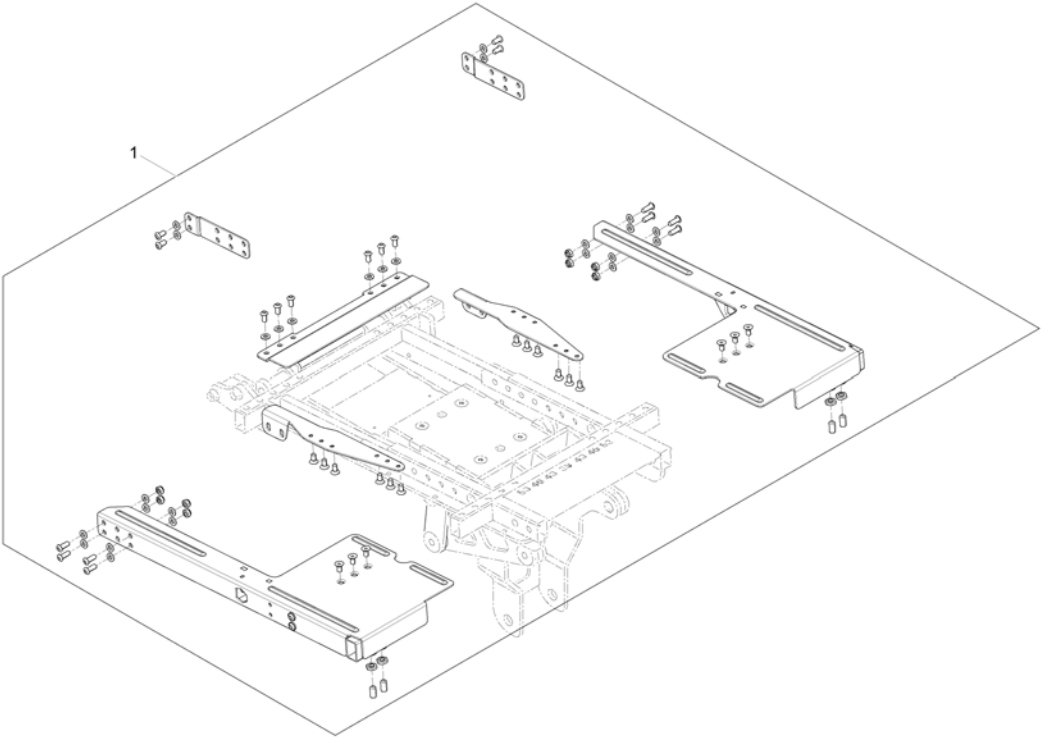


Pos.	Part number	Description	Valid from → until	Qty
1	SP1563052	Adapter Kit for Universal Seat		1


Seat Cushion

1973212/00

Matrix Flo-tech

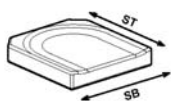

Onyx Cushion

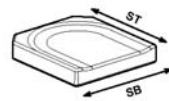

1	38cm	41cm	black smooth
2	38cm	46cm	
3	38cm	51cm	
4	43cm	41cm	
5	43cm	46cm	
6	43cm	51cm	
7	48cm	41cm	
8	48cm	46cm	
9	48cm	51cm	
10	53cm	43cm	
11	53cm	46cm	
12	53cm	51cm	
13	38cm	41cm	dartex grey
14	38cm	46cm	
15	38cm	51cm	
16	43cm	41cm	
17	43cm	46cm	
18	43cm	51cm	
19	48cm	41cm	
20	48cm	46cm	
21	48cm	51cm	
22	53cm	43cm	
23	53cm	46cm	
24	53cm	51cm	


25	38cm	41cm	grey
26	38cm	46cm	
27	38cm	51cm	
28	43cm	41cm	
29	43cm	46cm	
30	43cm	51cm	
31	48cm	41cm	
32	48cm	46cm	
33	48cm	51cm	
34	53cm	43cm	
35	53cm	46cm	
36	53cm	51cm	
37	38cm	41cm	black red seam
38	38cm	46cm	
39	38cm	51cm	
40	43cm	41cm	
41	43cm	46cm	
42	43cm	51cm	
43	48cm	41cm	
44	48cm	46cm	
45	48cm	51cm	
46	53cm	43cm	
47	53cm	46cm	
48	53cm	51cm	

49	40cm	40cm	black
50	40cm	45cm	
51	45cm	40cm	
52	45cm	45cm	
53	51cm	40cm	
54	51cm	45cm	

Pos.	Part number	Description	Valid from → until	Qty
1	SPFLB15/16	Flo-Tech Lite 15x16"		1
2	SPFLB15/18	Seatcushion Easy-Adapt SB38 x ST46cm black smooth		1
3	SPFLB15/20	Seatcushion Easy-Adapt SB38 x ST51cm black smooth		1
4	SPFLB17/16	Seatcushion Easy-Adapt SB43 x ST41cm black smooth		1
5	SPFLB17/18	Seatcushion Easy-Adapt SB43 x ST46cm black smooth		1
6	SPFLB17/20	Seatcushion Easy-Adapt SB43 x ST51cm black smooth		1
7	SPFLB19/16	Seatcushion Easy-Adapt SB48 x ST41cm black smooth		1
8	SPFLB19/18	Seatcushion Easy-Adapt SB48 x ST46cm black smooth		1
9	SPFLB19/20	Flo-Tech Lite 19x20"		1
10	SPFCB21/17	Flo-Tech Contour 21x17"		1
11	SPFCB21/18	Flo-Tech Contour 21x18"		1
12	SPFCB21/20	Flo-Tech Contour 21x20"		1
13	SP1525250-0232	Seatcushion Easy-Adapt SB38 x ST41cm dartex grey	→ 2014-11-30	1
13	SP1529546-TR26	seat cushion Flo-tech Lite SW 38 x SD 41 Dartex black	2014-12-01 →	1
14	SP1525252-0232	Seatcushion Easy-Adapt SB38 x ST46cm dartex grey	→ 2014-11-30	1
14	SP1529547-TR26	seat cushion Flo-tech Lite SW 38 x SD 46 Dartex black	2014-12-01 →	1
15	SP1525253-0232	Seatcushion Easy-Adapt SB38 x ST51cm dartex grey	→ 2014-11-30	1
15	SP1529548-TR26	seat cushion Flo-tech Lite SW 38 x SD 51 Dartex black	2014-12-01 →	1
16	SP1525254-0232	Seatcushion Easy-Adapt SB43 x ST41cm dartex grey	→ 2014-11-30	1
16	SP1529549-TR26	seat cushion Flo-tech Lite SW 43 x SD 41 Dartex black	2014-12-01 →	1
17	-	No longer available - please choose Dartex black	→ 2014-11-30	1
17	SP1529550-TR26	seat cushion Flo-tech Lite SW 43 x SD 46 Dartex black	2014-12-01 →	1
18	SP1525256-0232	Seatcushion Easy-Adapt SB43 x ST51cm dartex grey	→ 2014-11-30	1
18	SP1529551-TR26	seat cushion Flo-tech Lite SW 43 x SD 51 Dartex black	2014-12-01 →	1
19	SP1525257-0232	Seatcushion Easy-Adapt SB48 x ST41cm dartex grey	→ 2014-11-30	1
19	SP1529552-TR26	seat cushion Flo-tech Lite SW 48 x SD 41 Dartex black	2014-12-01 →	1
20	SP1525258-0232	Seatcushion Easy-Adapt SB48 x ST46cm dartex grey	→ 2014-11-30	1
20	SP1529553-TR26	seat cushion Flo-tech Lite SW 48 x SD 46 Dartex black	2014-12-01 →	1
21	SP1525259-0232	Easy-Adapt cushion (Flo-tech Lite) SW 48 x SD 51cm Dartex	→ 2014-11-30	1
21	SP1529554-TR26	seat cushion Flo-tech Lite SW 48 x SD 51 Dartex black	2014-12-01 →	1
22	SP1525260-0232	Seatcushion Easy-Adapt SB53 x ST43cm dartex grey	→ 2014-11-30	1


Seat Cushion

Pos.	Part number	Description	Valid from → until	Qty
22	SP1529555-TR26	seat cushion Flo-tech Contour SW 53 x SD 43 Dartex black	2014-12-01 →	1
23	SP1525261-0232	Seatcushion Easy-Adapt SB53 x ST46cm dartex grey	→ 2014-11-30	1
23	SP1529556-TR26	seat cushion Flo-tech Contour SW 53 x SD 46 Dartex black	2014-12-01 →	1
24	SP1525262-0232	Seatcushion Easy-Adapt SB53 x ST51cm dartex grey	→ 2014-11-30	1
24	SP1529570-TR26	seat cushion Flo-tech Contour SW 53 x SD 51 Dartex black	2014-12-01 →	1
25	SP1525250-0237	Seatcushion Easy-Adapt SB38 x ST41cm grey plush		1
26	SP1525252-0237	Seatcushion Easy-Adapt SB38 x ST46cm grey plush		1
27	SP1525253-0237	Seatcushion Easy-Adapt SB38 x ST51cm grey plush		1
28	SP1525254-0237	Seatcushion Easy-Adapt SB43 x ST41cm grey plush		1
29	SP1525255-0237	Seatcushion Easy-Adapt SB43 x ST46cm grey plush		1
30	SP1525256-0237	Seatcushion Easy-Adapt SB43 x ST51cm grey plush		1
31	SP1525257-0237	Seatcushion Easy-Adapt SB48 x ST41cm grey plush		1
32	SP1525258-0237	Seatcushion Easy-Adapt SB48 x ST46cm grey plush		1
33	SP1525259-0237	Easy-Adapt cushion (Flo-tech Lite) SW 48 x SD 51cm Grey Plush		1
34	SP1525260-0237	Seatcushion Easy-Adapt SB53 x ST43cm grey plush		1
35	SP1525261-0237	Seatcushion Easy-Adapt SB53 x ST46cm grey plush		1
36	SP1525262-0237	Seatcushion Easy-Adapt SB53 x ST51cm grey plush		1
37	SP1529546	Seatcushion Easy-Adapt SB38 x ST41cm black with red seam		1
38	SP1529547	Seatcushion Easy-Adapt SB38 x ST46cm black with red seam		1
39	SP1529548	Seatcushion Easy-Adapt SB38 x ST51cm black with red seam		1
40	SP1529549	Seatcushion Easy-Adapt SB43 x ST41cm black with red seam		1
41	SP1529550	Seatcushion Easy-Adapt SB43 x ST46cm black with red seam		1
42	SP1529551	Seatcushion Easy-Adapt SB43 x ST51cm black with red seam		1
43	SP1529552	Seatcushion Easy-Adapt SB48 x ST41cm black with red seam		1
44	SP1529553	Seatcushion Easy-Adapt SB48 x ST46cm black with red seam		1
45	SP1529554	Seatcushion Easy-Adapt SB48 x ST51cm black with red seam		1
46	SP1529555	Seatcushion Easy-Adapt SB53 x ST43cm black with red seam		1
47	SP1529556	Seatcushion Easy-Adapt SB53 x ST46cm black with red seam		1
48	SP1529570	Seatcushion Easy-Adapt SB53 x ST51cm black with red seam		1
49	SP1532058	ONYX Seat Cushion 40x40cm (16"x16")		1
50	SP1534080	ONYX Seat Cushion 40x45cm (16"x18")		1
51	SP1532059	ONYX Seat Cushion 45x40cm (18"x16")		1
52	SP1532060	ONYX Seat Cushion 45x45cm (18"x18")		1
53	SP1534136	ONYX Seat Cushion 51x40cm (20"x16")		1
54	SP1534199	ONYX Seat Cushion 51x45cm (20"x18")		1

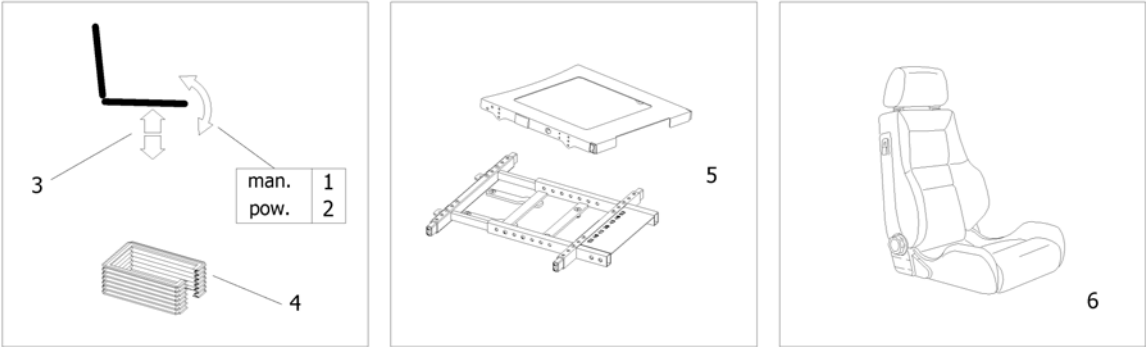

Pos.	Part number	Description	Valid from → until	Qty
1	-	Seatsupport, manuell [1023/1025]		1
2	-	Seattilt 30° [1014]		1
3	-	Lifter [1030]		1
4	-	Bellow [1311]		1
5	-	Seat Frame & Plate for Recaro Seat		1
6	-	Recaro Seat		1


Seat Frame & Plate for Recaro Seat

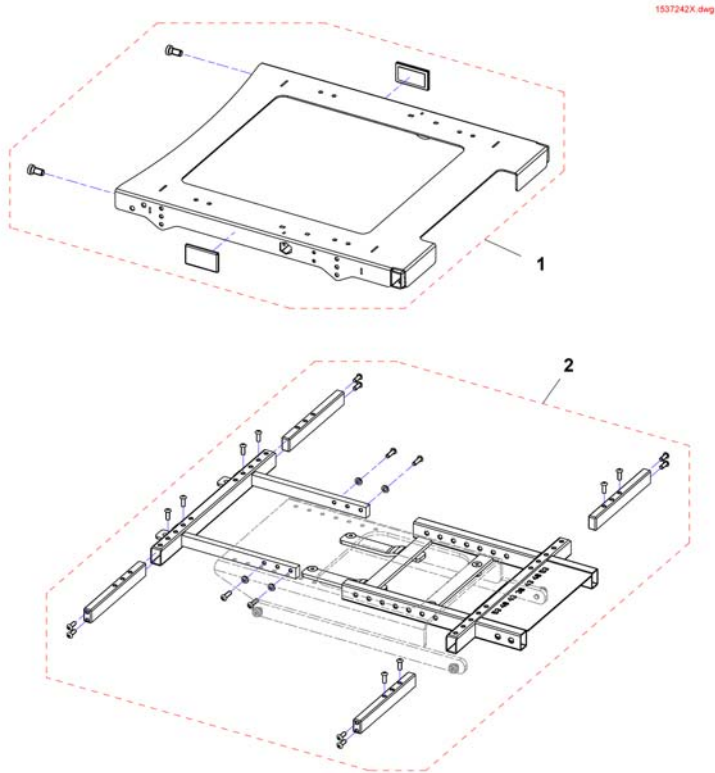

Pos.	Part number	Description	Valid from → until	Qty
1	SP1530519	Seatplate Recaro		1
2	SP1541739	Frame for recaroseatplate		1

Recaro Seat



1422546.dwg

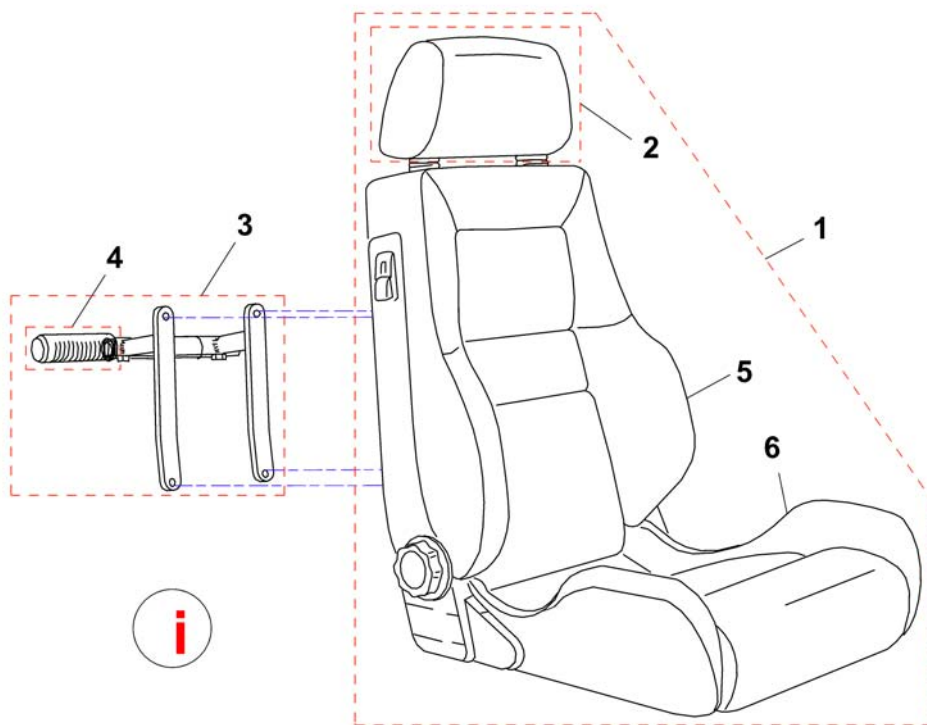


Pos.	Part number	Description	Valid from → until	Qty
i	-	Recaro Mobility 24V is no longer available. Please pay attention to the change list!		1
i	-	Change list Recaro 24V		1
	F367002	Belt with Velcro fastener for Recaro		1
	Z163001A	Belt with snap lock & Velcro fastener for Recaro Seat		1
	SP1423173	RECARO SEAT MOUNTING KIT		1
1	SP1552597	Recaroseat Mobilty, 12V, cpl.		1
1	SP1422546	Recaro Seat N-Joy cpl.		1
2	N89999	HEADREST - SPORT NARDO		1
3	SP1423177	HOLDER F. PUSH HANDLE RECARO CPL.		1
4	1435421	Handle plastic black Ø22 "Invacare"		1
5	SP1552596	Backrest, Recaro Mobility, 12v, cpl. <i>Recaro Mobility 24V is no longer available. Please pay attention to the compatibility chart!</i>		1
5	SP1552781	Backrest Recaro Mobility, mechanic		1
6	SP1419207	Seatpart Recaro Mobility		1

Change list Recaro 24V

1420495X.iso

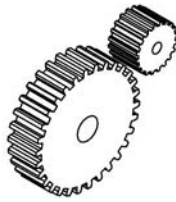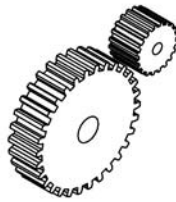
	 		 	
Storm4	O	O	P	P
G50	O	O	P	special
FDX	O	O	P	special
Typhoon	O	P	O	O
Typhoon II	O	P	O	O
Zephyr	O	P	O	O
Tornado	O	P	O	O
Storm3	O	P	O	O

Pos.	Part number	Description	Valid from → until	Qty
-	-	For further information please contact your customer service.		1

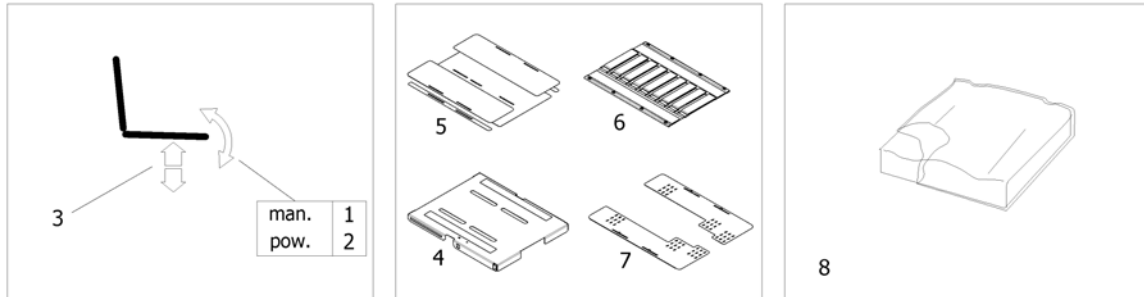


Pos.	Part number	Description	Valid from → until	Qty
1	-	Tilt 0°, 4°, 8° [1091]		1
2	-	Tilt 30° [1093]		1
3	-	Lifter [1094]		1
4	-	One Piece Seat Plate [2110]		1
5	-	Telescopic Seat Frame and Plate [2111]		1
6	-	Telescopic Seat Frame and Sling Seat [2112]		1
7	-	Telescopic Seat Frame and Universal Adapter [2109]		1
8	-	Seat Cushions "Standard", "Matrx Flo-Tech" & "Matrx Vi"		1
8	-	Seat Cushions "Matrx Libra"		1

1573383 iso

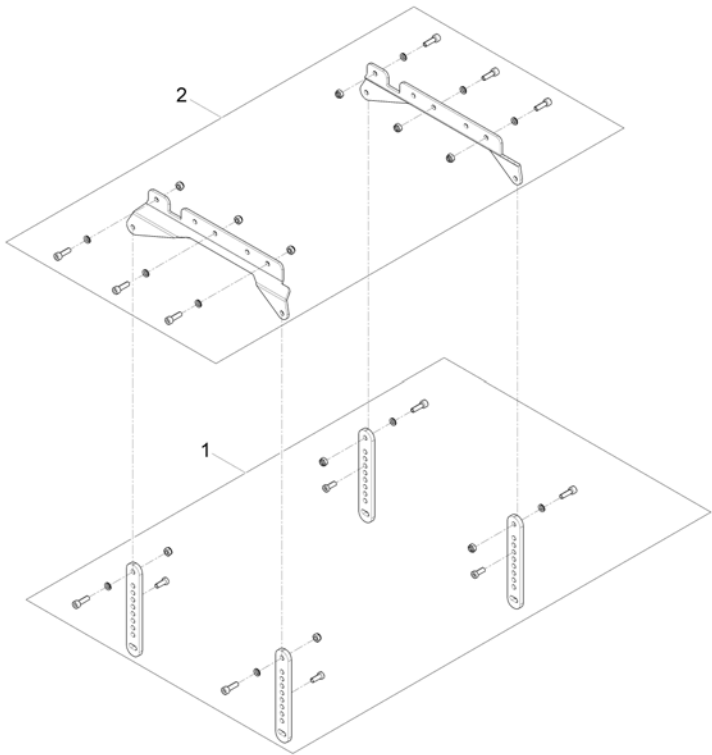

Pos.	Part number	Description	Valid from → until	Qty
1	SP1574093	Seat Posts		1
2	SP1574094	Mounting Rails		1


1573340 iso

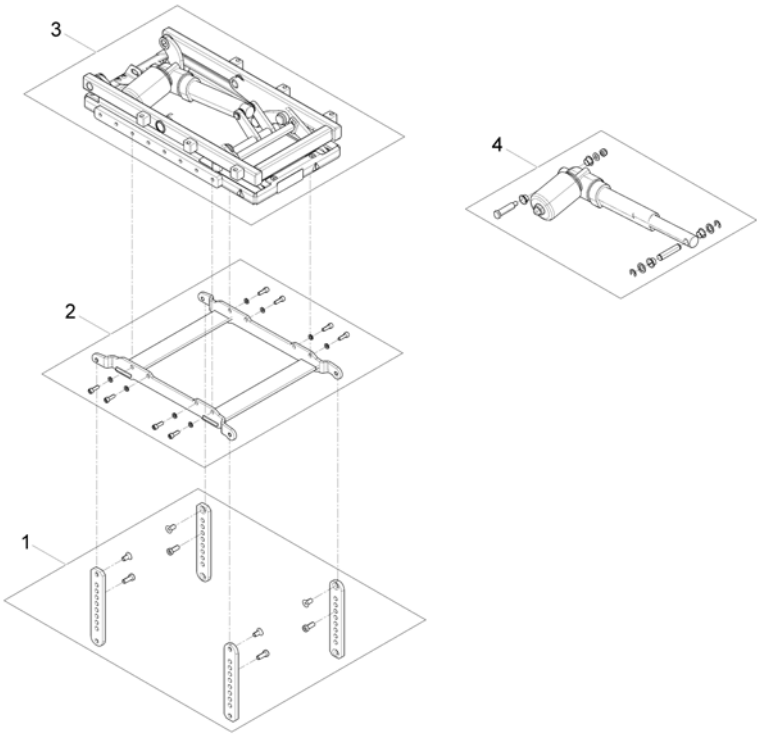


Pos.	Part number	Description	Valid from → until	Qty
1	SP1574100	Seat Posts		1
2	SP1574101	Lifter / Tilt Bracket		1
3	SP1558714	Tilt unit		1
4	SP1571809	Actuator		1

Lifter

1573339 iso

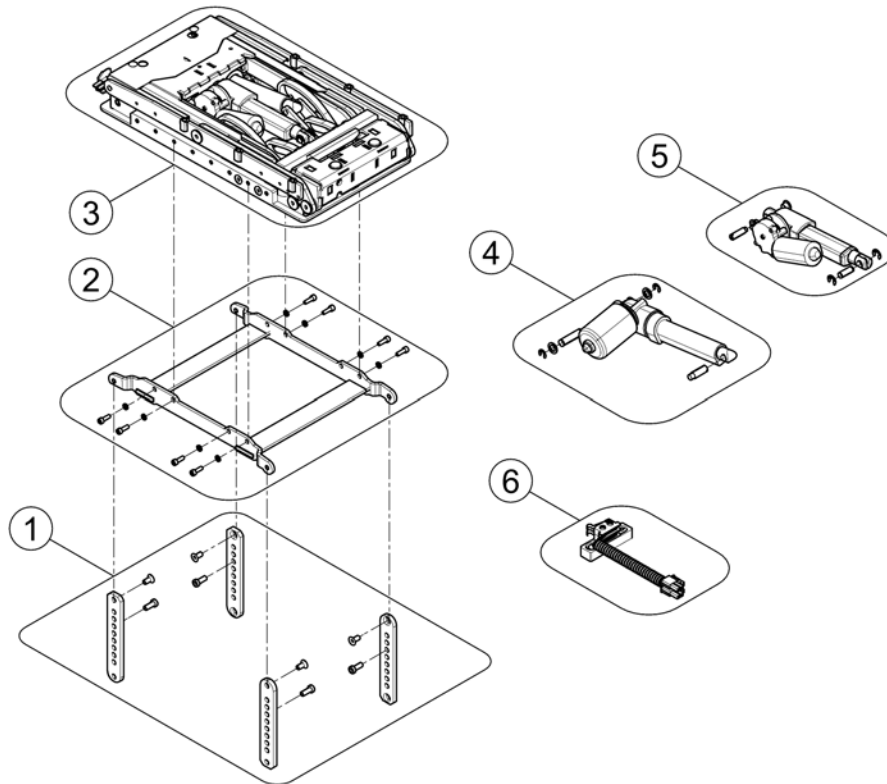


Pos.	Part number	Description	Valid from → until	Qty
1	SP1574100	Seat Posts		1
2	SP1574101	Lifter / Tilt Bracket		1
3	SP1558715	Liter- / Tilt unit		1
4	SP1571823	Lifter actuator		1
5	SP1571822	Tilt actuator		1
6	SP1577762	Slow-Stop Switchcable with Mounting		1

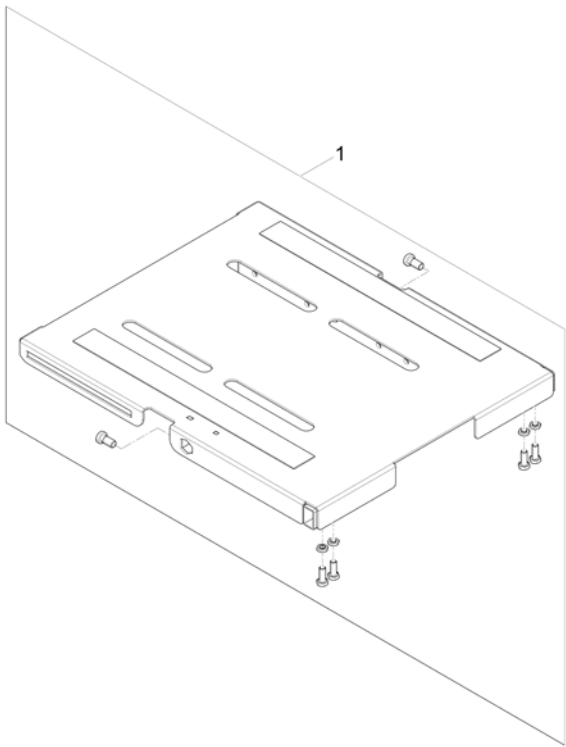

Pos.	Part number	Description	Valid from → until	Qty
1	SP1571297	One Piece Seat Plate		1


Telescopic Seat Frame and Plate

1572312 iso

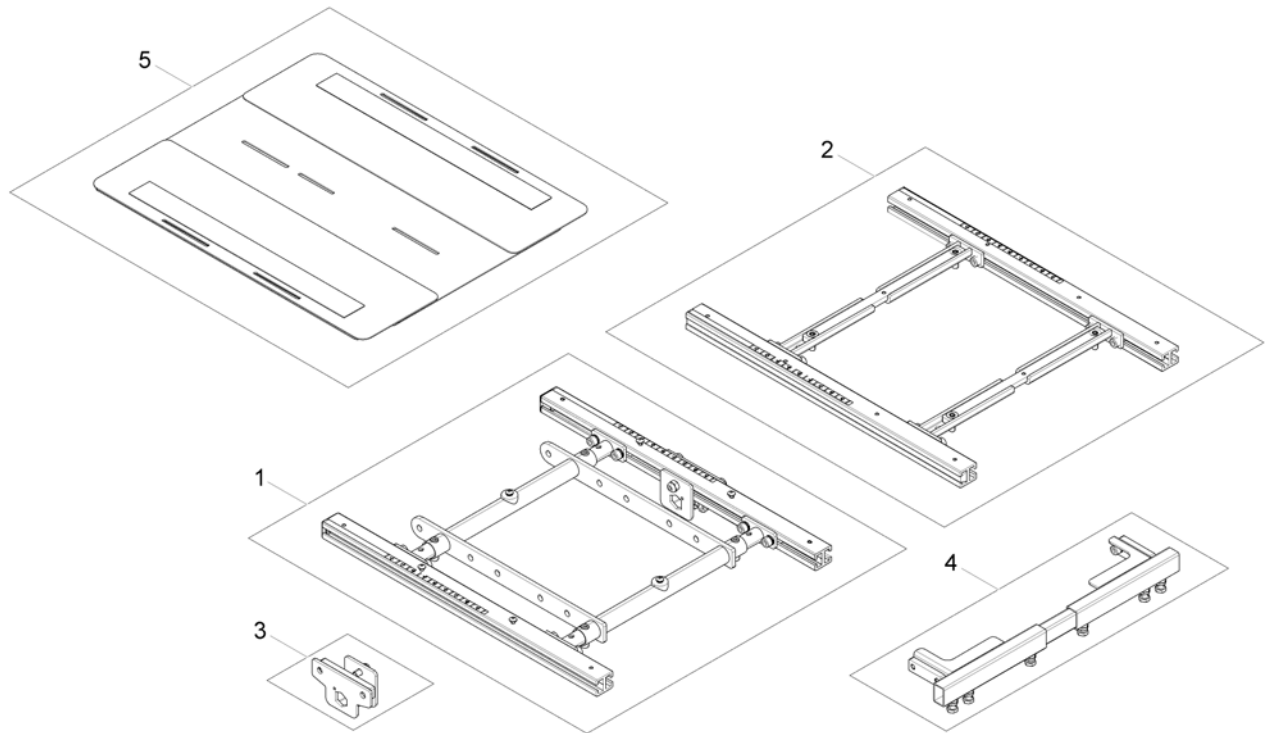

Pos.	Part number	Description	Valid from → until	Qty
1	-	Telescopic Seat Frame for Tilt 0°, 4°, 8°		1
2	-	Telescopic Seat Frame for Tilt 30° / Lifter		1
3	-	Armrest bracket		1
4	-	Legrest Support		1
5	-	Seat Plate		1


Telescopic Seat Frame for Tilt 0°, 4°, 8°

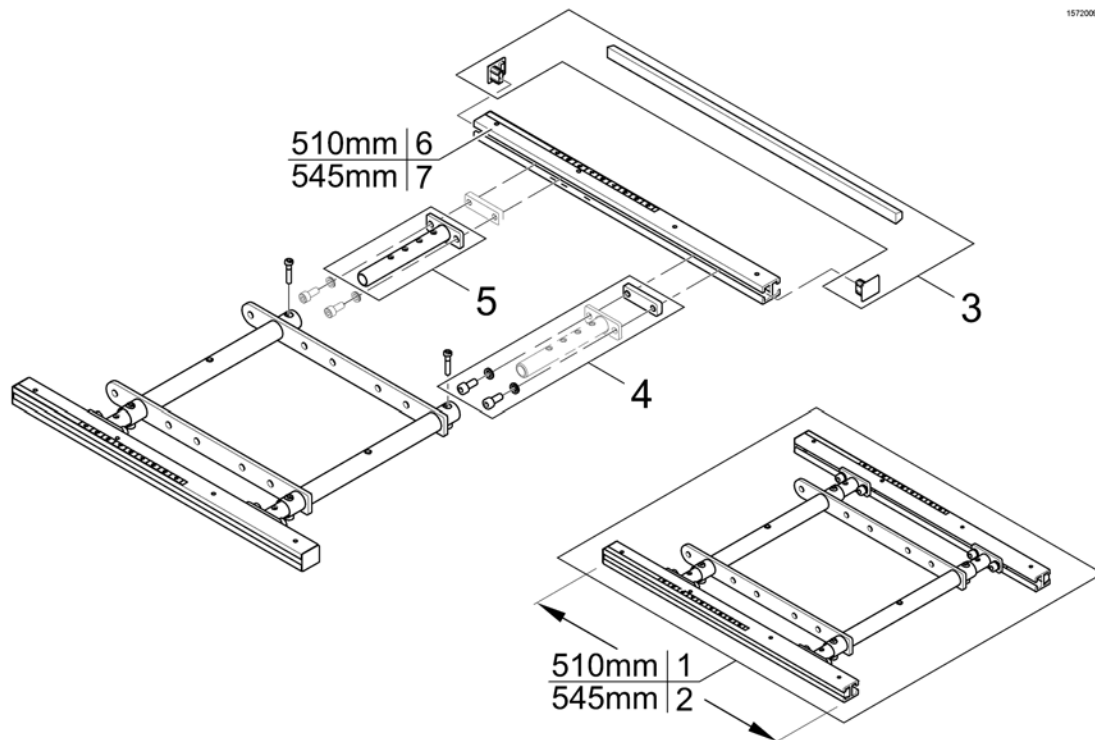

Pos.	Part number	Description	Valid from → until	Qty
1	SP1572977	Seatframe 510mm, cpl., pre set up to SW 43cm		1
2	SP1582216	Seatframe 545mm, cpl., pre set up to SW 43cm		1
3	SP1571810	Sealing strap with plugs		1
4	SP1571811	Groove stone with mounting parts		1
5	SP1554646	Adjustment Tube		1
6	SP1588443	Seat Rails 510mm (2pcs kit)		1
7	SP1588444	Seat Rails 545mm (2pcs kit)		1


Telescopic Seat Frame for Tilt 30° / Lifter

1572004 Rev

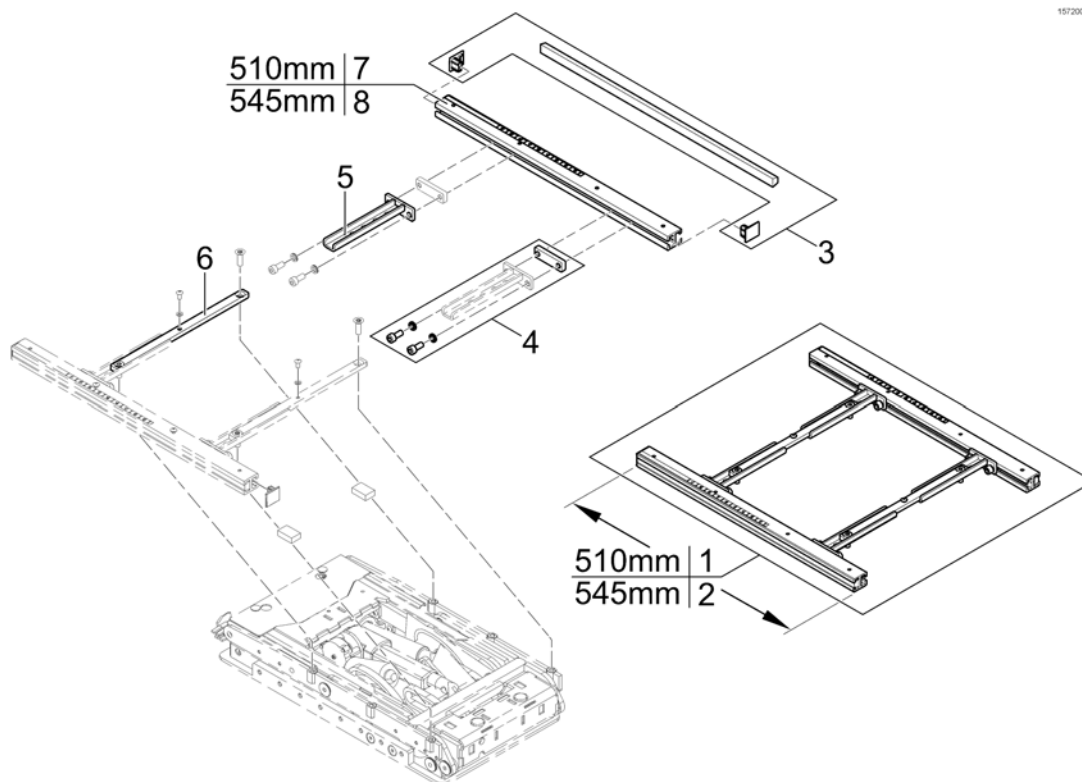


Pos.	Part number	Description	Valid from → until	Qty
1	SP1572992	Seatframe 510mm, pre set up to SW 430mm, cpl.		1
2	SP1582219	Seatframe 545mm, pre set up to SW 430mm, cpl.		1
3	SP1571810	Sealing strap with plugs		1
4	SP1571811	Groove stone with mounting parts		1
5	SP1567007	Seat Frame Adapter		1
6	SP1567008	Middle Rail		2
7	SP1588443	Seat Rails 510mm (2pcs kit)		1
8	SP1588444	Seat Rails 545mm (2pcs kit)		1

1571834 (6)

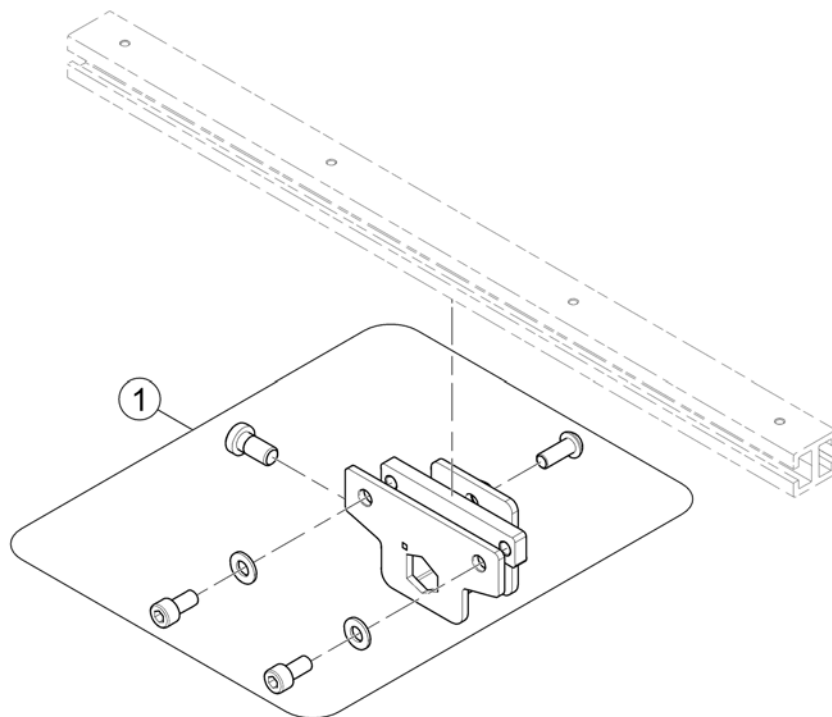


Pos.	Part number	Description	Valid from → until	Qty
1	SP1570189	Armrest bracket		1

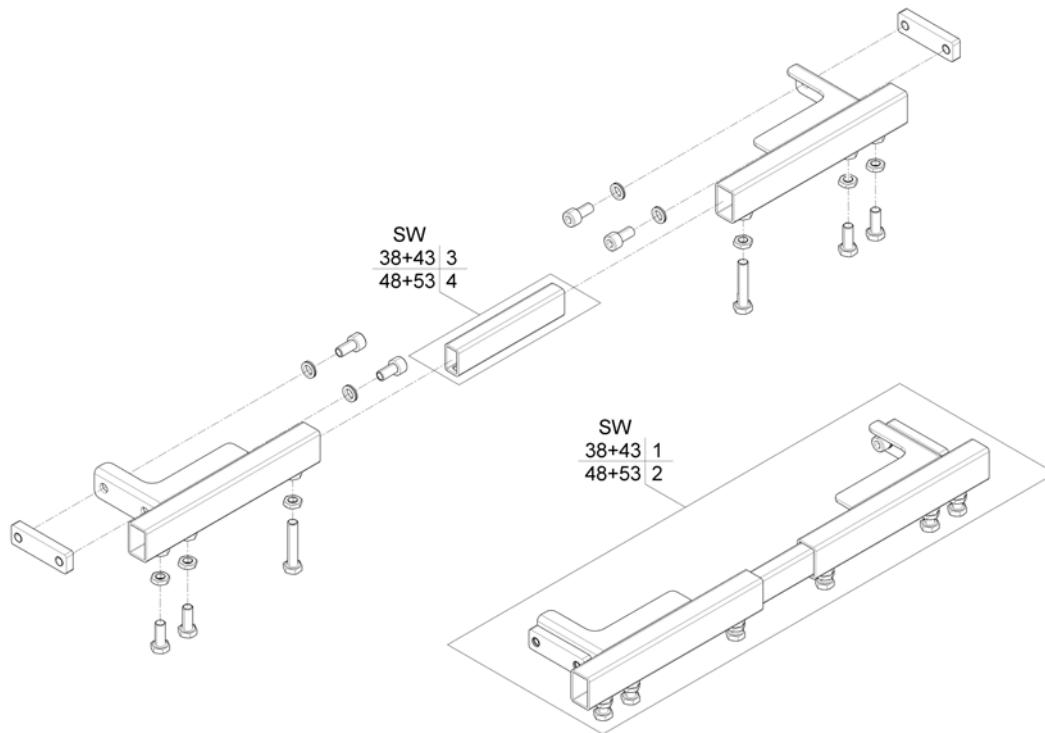

Pos.	Part number	Description	Valid from → until	Qty
1	SP1558935	Legrest Support SW38+43, cpl.		1
2	SP1559294	Legrest Support SW48+53, cpl.		1
3	SP1558134	Connection Tube SW380+430mm		1
4	SP1558135	Connection Tube SW480+530mm		1


1572310.kao

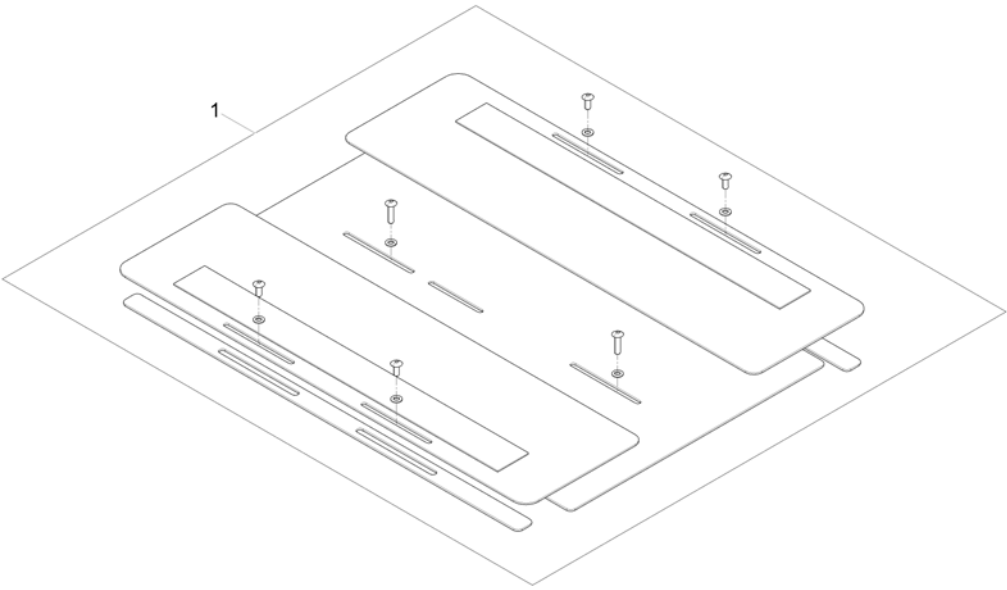

Pos.	Part number	Description	Valid from → until	Qty
1	SP1572199	Seat Plate 510mm, cpl.		1

Telescopic Seat Frame and Sling Seat


1572311.00

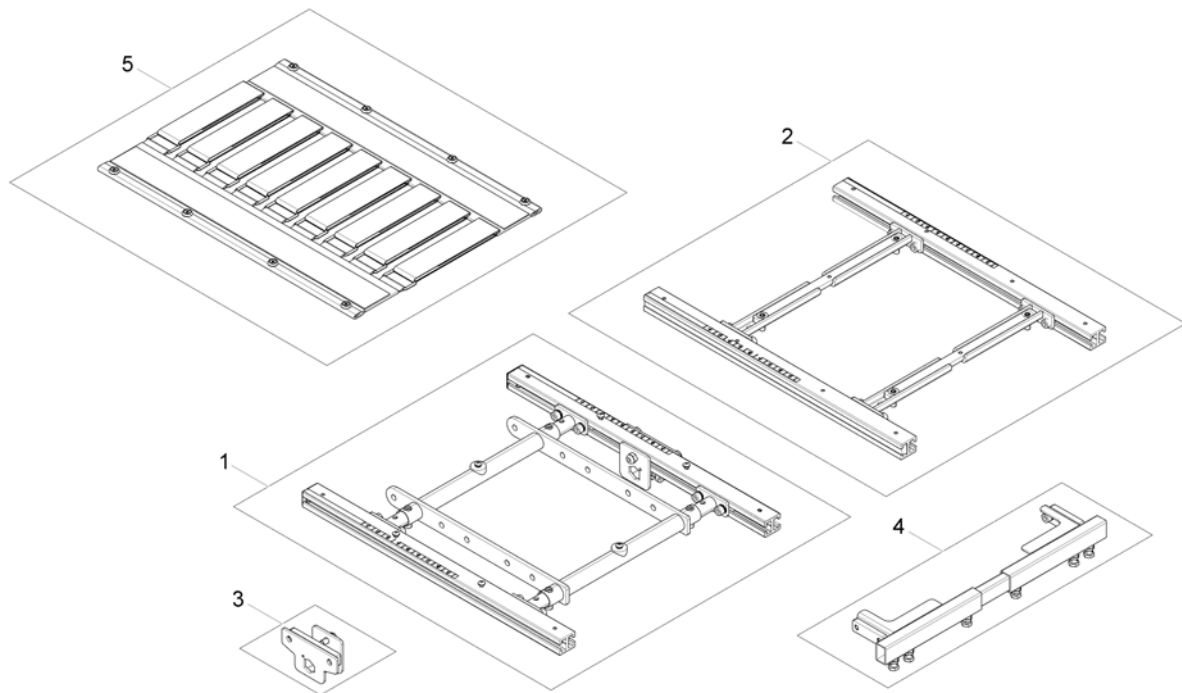

Pos.	Part number	Description	Valid from → until	Qty
1	-	Telescopic Seat Frame for Tilt 0°, 4°, 8°		1
2	-	Telescopic Seat Frame for Tilt 30° / Lifter		1
3	-	Armrest bracket		1
4	-	Legrest Support		1
5	-	Sling Seat		1


Telescopic Seat Frame for Tilt 0°, 4°, 8°

1571950-000

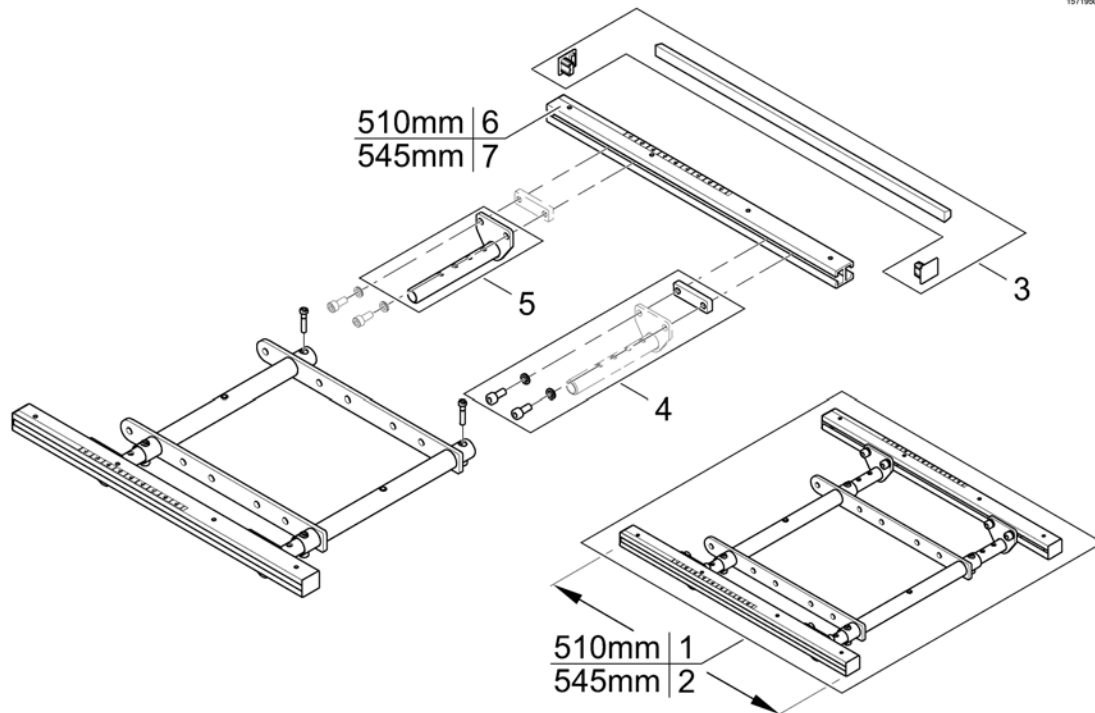


Pos.	Part number	Description	Valid from → until	Qty
1	SP1572978	Seatframe 510mm, cpl., pre set up to SW 43cm		1
2	SP1582217	Seatframe 545mm, cpl., pre set up to SW 43cm		1
3	SP1571810	Sealing strap with plugs		1
4	SP1571811	Groove stone with mounting parts		1
5	SP1555400	Adjustment Tube		1
6	SP1588443	Seat Rails 510mm (2pcs kit)		1
7	SP1588444	Seat Rails 545mm (2pcs kit)		1


Telescopic Seat Frame for Tilt 30° / Lifter

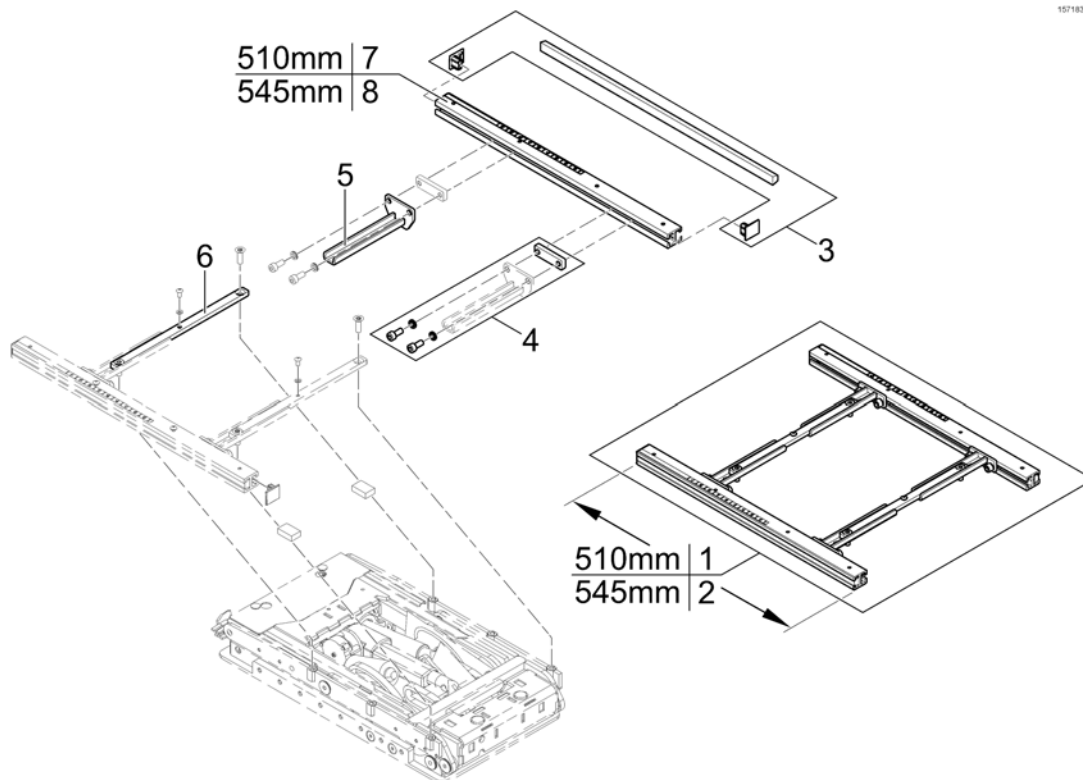

Pos.	Part number	Description	Valid from → until	Qty
1	SP1572993	Seatframe 510mm, pre set up to SW 430mm, cpl.		1
2	SP1582230	Seatframe 545mm, pre set up to SW 430mm, cpl.		1
3	SP1571810	Sealing strap with plugs		1
4	SP1571811	Groove stone with mounting parts		1
5	SP1567720	Seatframe Adapter		1
6	SP1567008	Middle Rail		1
7	SP1588443	Seat Rails 510mm (2pcs kit)		1
8	SP1588444	Seat Rails 545mm (2pcs kit)		1

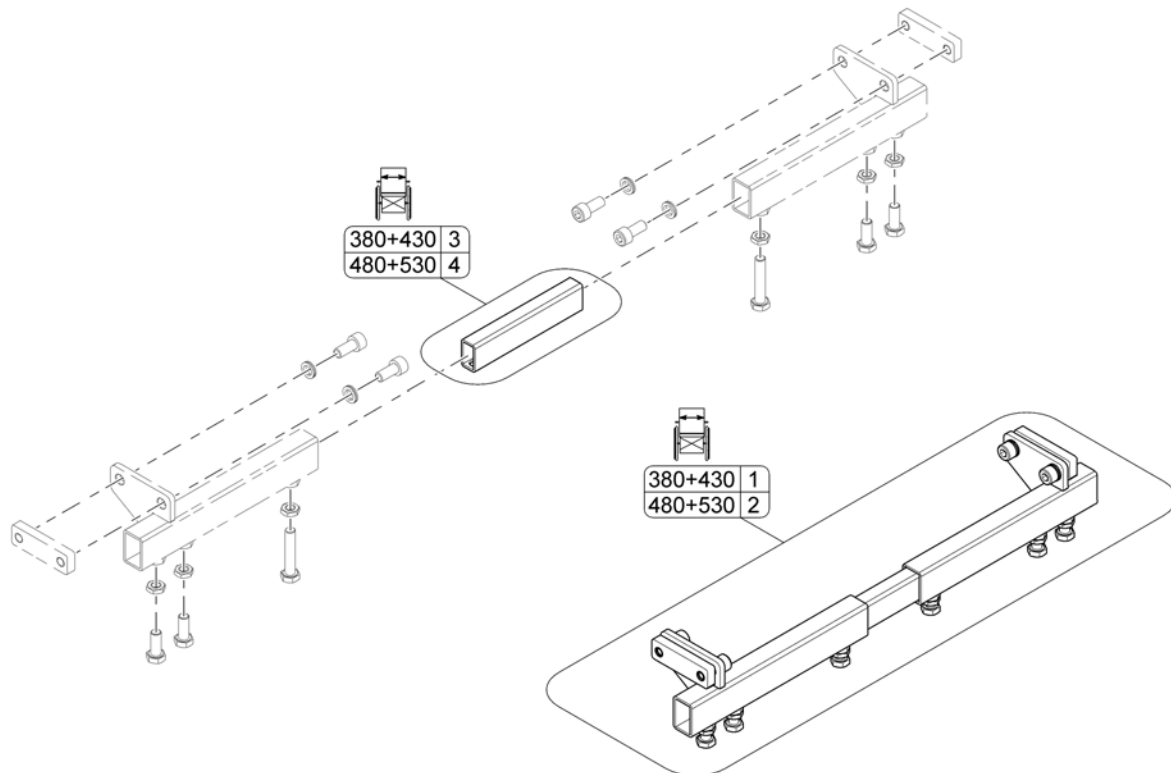


Pos.	Part number	Description	Valid from → until	Qty
1	SP1559305	Legrest Support SW380+430mm, cpl.		1
2	SP1559307	Legrest Support SW480+530mm, cpl.		1
3	SP1558134	Connection Tube SW380+430mm		1
4	SP1558135	Connection Tube SW480+530mm		1

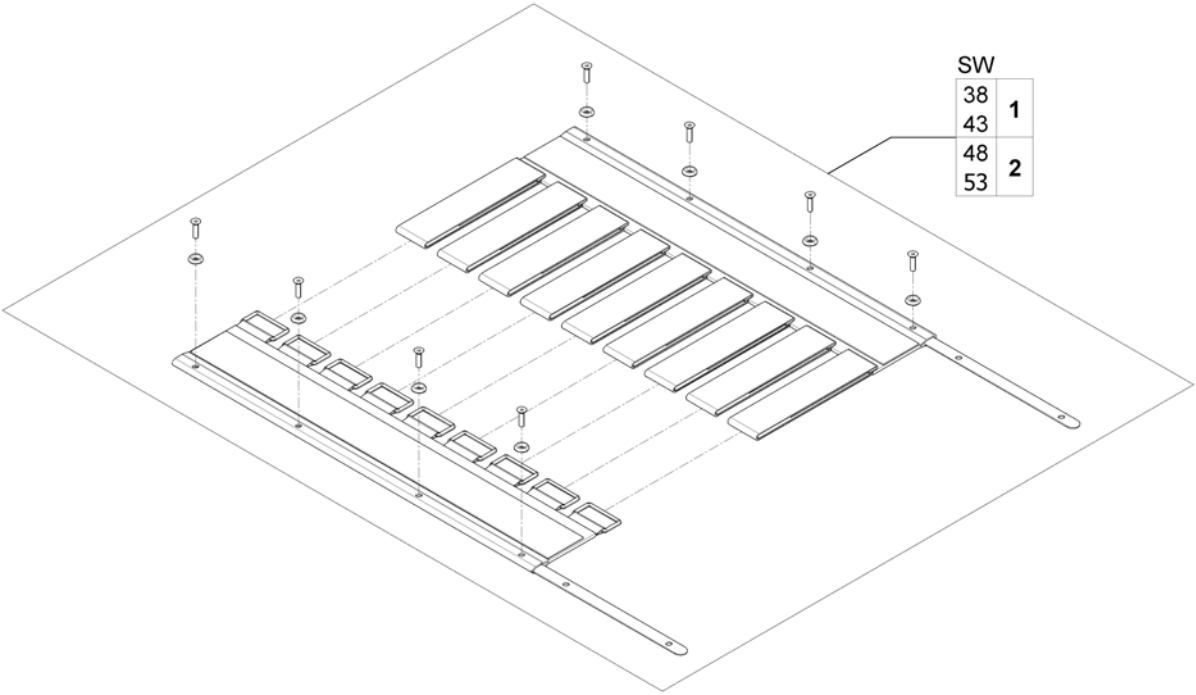

Pos.	Part number	Description	Valid from → until	Qty
1	SP1557877	Sling Seat SW380/430, 510mm		1
2	SP1555911	Sling Seat SW480/530, 510mm		1

Telescopic Seat Frame and Universal Adapter


1581864 iso

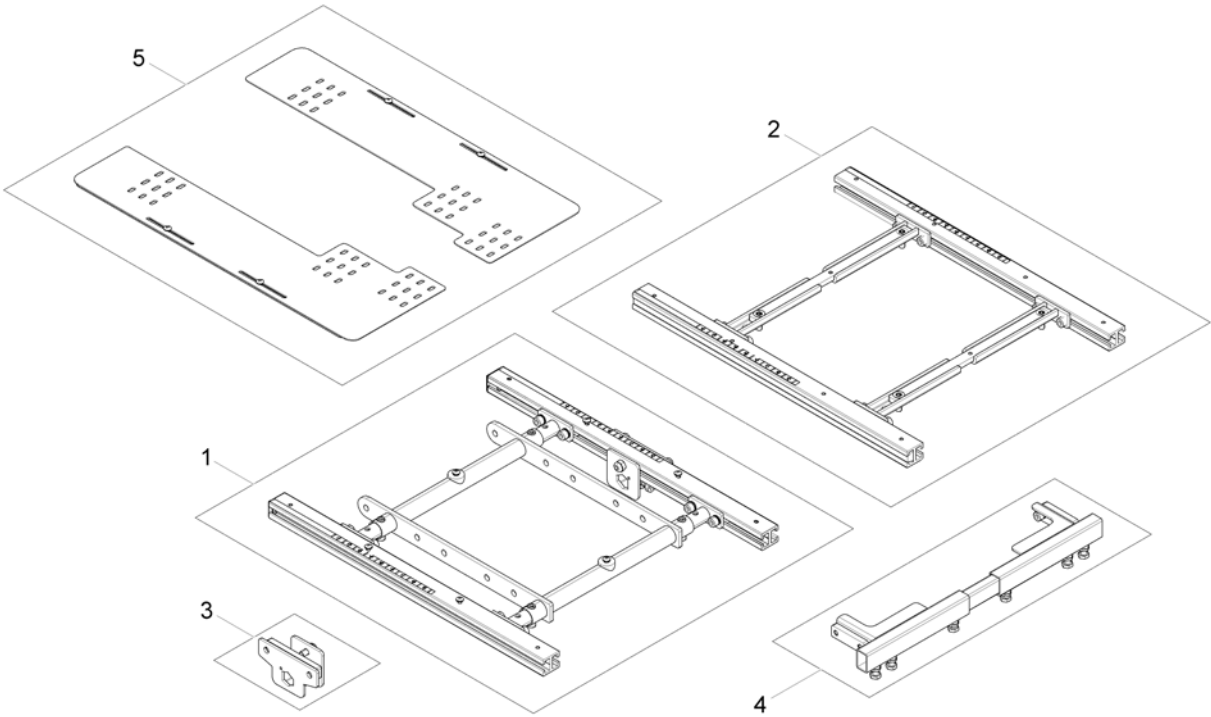


Pos.	Part number	Description	Valid from → until	Qty
1	-	Telescopic Seat Frame for Tilt 12° / 20°		1
2	-	Telescopic Seat Frame for Tilt 30° / Lifter		1
3	-	Armrest bracket		1
4	-	Legrest Support		1
5	-	Adapter Kit for Universal Seat		1

Telescopic Seat Frame for Tilt 12° / 20°

1572008-000

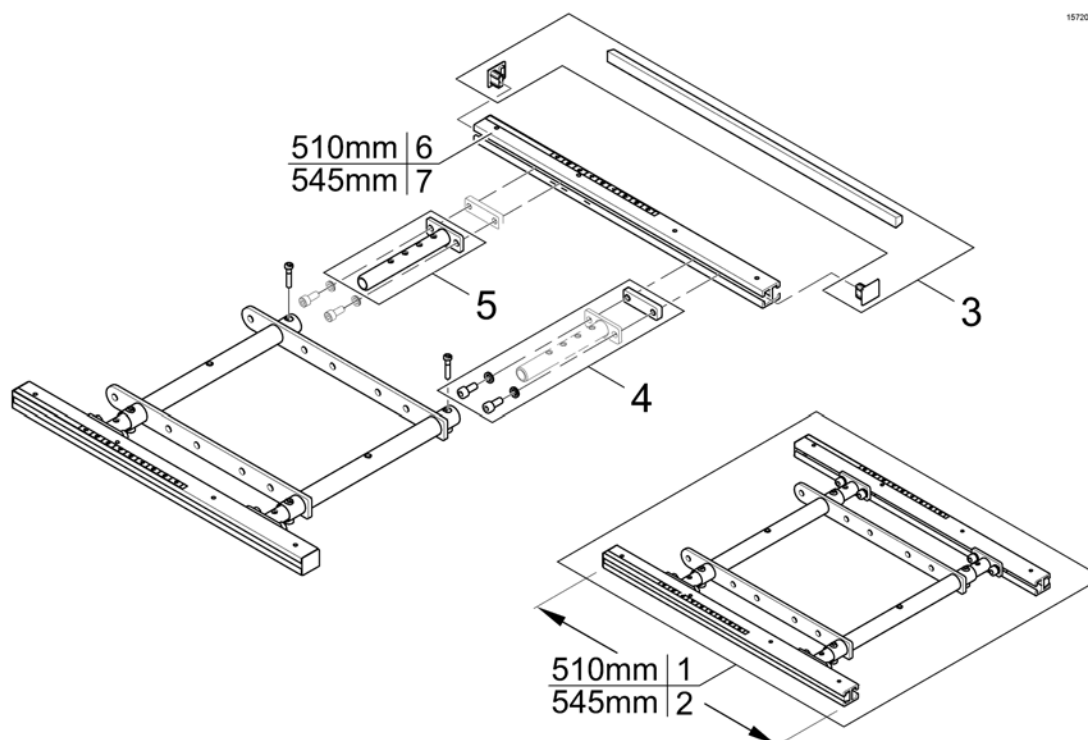


Pos.	Part number	Description	Valid from → until	Qty
1	SP1572977	Seatframe 510mm, cpl., pre set up to SW 43cm		1
2	SP1582216	Seatframe 545mm, cpl., pre set up to SW 43cm		1
3	SP1571810	Sealing strap with plugs		1
4	SP1571811	Groove stone with mounting parts		1
5	SP1554646	Adjustment Tube		1
6	SP1588443	Seat Rails 510mm (2pcs kit)		1
7	SP1588444	Seat Rails 545mm (2pcs kit)		1

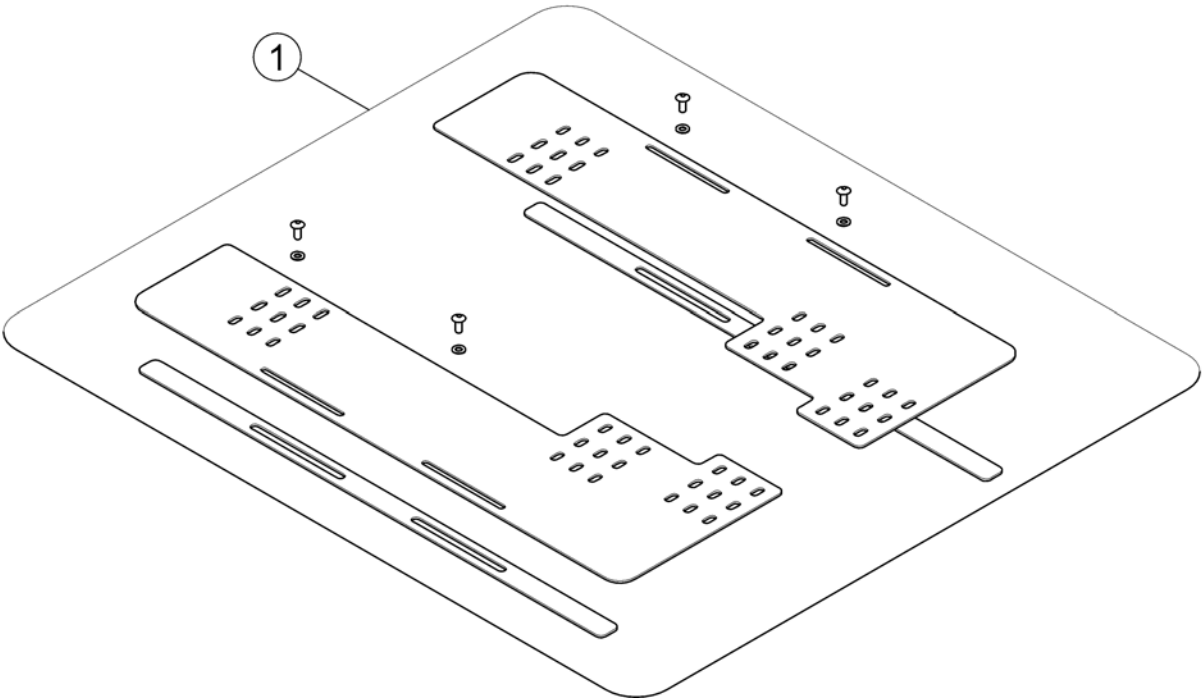

Pos.	Part number	Description	Valid from → until	Qty
1	SP1580750	Adapter Kit for Universal Seat 510mm		1

Seat Cushions "Standard", "Matrx Flo-Tech" & "Matrx Vi"

1573000 iso

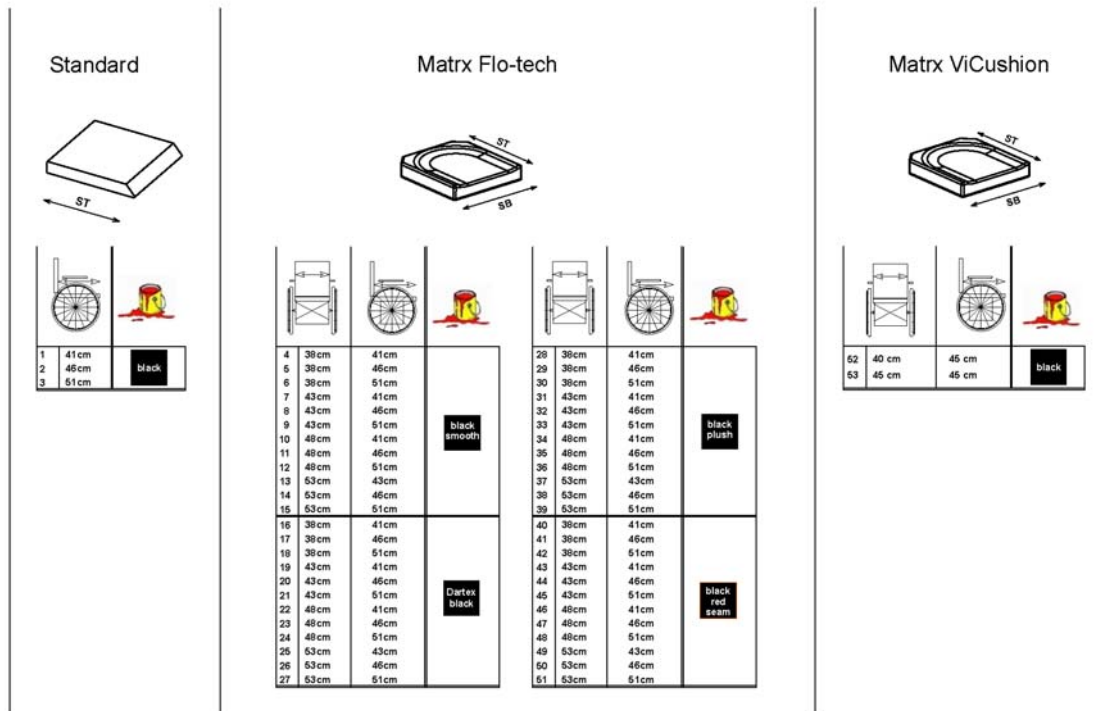

Pos.	Part number	Description	Valid from → until	Qty
1	1420460	Seat Cushion Standard - SW 430mm, SD 410mm [2141]		1
2	1420461	Seat Cushion Standard - SW 430mm, SD 460mm [2141]		1
3	1420462	Seat Cushion Standard - SW 430mm, SD 510mm [2141]		1
4	SPFLB15/16	Flo-Tech Lite 15x16" [2142+2151]		1
5	SPFLB15/18	Seatcushion Easy-Adapt SB38 x ST46cm black smooth [2142+2151]		1
6	SPFLB15/20	Seatcushion Easy-Adapt SB38 x ST51cm black smooth [2142+2151]		1
7	SPFLB17/16	Seatcushion Easy-Adapt SB43 x ST41cm black smooth [2142+2151]		1
8	SPFLB17/18	Seatcushion Easy-Adapt SB43 x ST46cm black smooth [2142+2151]		1
9	SPFLB17/20	Seatcushion Easy-Adapt SB43 x ST51cm black smooth [2142+2151]		1
10	SPFLB19/16	Seatcushion Easy-Adapt SB48 x ST41cm black smooth [2142+2151]		1
11	SPFLB19/18	Seatcushion Easy-Adapt SB48 x ST46cm black smooth [2142+2151]		1
12	SPFLB19/20	Flo-Tech Lite 19x20" [2142+2151]		1
13	SPFCB21/17	Flo-Tech Contour 21x17" [2142+2151]		1
14	SPFCB21/18	Flo-Tech Contour 21x18" [2142+2151]		1
15	SPFCB21/20	Flo-Tech Contour 21x20" [2142+2151]		1
16	SP1529546-TR26	seat cushion Flo-tech Lite SW 38 x SD 41 Dartex black [2142+2152]		1
17	SP1529547-TR26	seat cushion Flo-tech Lite SW 38 x SD 46 Dartex black [2142+2152]		1
18	SP1529548-TR26	seat cushion Flo-tech Lite SW 38 x SD 51 Dartex black [2142+2152]		1

Seat Cushions "Standard", "Matrx Flo-Tech" & "Matrx Vi"


Pos.	Part number	Description	Valid from → until	Qty
19	SP1529549-TR26	seat cushion Flo-tech Lite SW 43 x SD 41 Dartex black [2142+2152]		1
20	SP1529550-TR26	seat cushion Flo-tech Lite SW 43 x SD 46 Dartex black [2142+2152]		1
21	SP1529551-TR26	seat cushion Flo-tech Lite SW 43 x SD 51 Dartex black [2142+2152]		1
22	SP1529552-TR26	seat cushion Flo-tech Lite SW 48 x SD 41 Dartex black [2142+2152]		1
23	SP1529553-TR26	seat cushion Flo-tech Lite SW 48 x SD 46 Dartex black [2142+2152]		1
24	SP1529554-TR26	seat cushion Flo-tech Lite SW 48 x SD 51 Dartex black [2142+2152]		1
25	SP1529555-TR26	seat cushion Flo-tech Contour SW 53 x SD 43 Dartex black [2142+2152]		1
26	SP1529556-TR26	seat cushion Flo-tech Contour SW 53 x SD 46 Dartex black [2142+2152]		1
27	SP1529570-TR26	seat cushion Flo-tech Contour SW 53 x SD 51 Dartex black [2142+2152]		1
28	SP1529546-TR35	seat cushion Flo-tech Lite SW 38 x SD 41 black chenille fabric [2142+2154]		1
29	SP1529547-TR35	seat cushion Flo-tech Lite SW 38 x SD 46 black chenille fabric [2142+2154]		1
30	SP1529548-TR35	seat cushion Flo-tech Lite SW 38 x SD 51 black chenille fabric [2142+2154]		1
31	SP1529549-TR35	seat cushion Flo-tech Lite SW 43 x SD 41 black chenille fabric [2142+2154]		1
32	SP1529550-TR35	seat cushion Flo-tech Lite SW 43 x SD 46 black chenille fabric [2142+2154]		1
33	SP1529551-TR35	seat cushion Flo-tech Lite SW 43 x SD 51 black chenille fabric [2142+2154]		1
34	SP1529552-TR35	Seat cushion Flo-tech Lite SW 48 x SD 41 black chenille fabric [2142+2154]		1
35	SP1529553-TR35	seat cushion Flo-tech Lite SW 48 x SD 46 black chenille fabric [2142+2154]		1
36	SP1529554-TR35	seat cushion Flo-tech Lite SW 48 x SD 51 black chenille fabric [2142+2154]		1
37	SP1529555-TR35	seat cushion Flo-tech Contour SW 53 X SD 43 black chenille fabric [2142+2154]		1
38	SP1529556-TR35	seat cushion Flo-tech Contour SW 53 x SD 46 black chenille fabric [2142+2154]		1
39	SP1529570-TR35	seat cushion Flo-tech Contour SW 53 x SD 51 black chenille fabric [2142+2154]		1
40	SP1529546	Seatcushion Easy-Adapt SB38 x ST41cm black with red seam [2142+2153]		1
41	SP1529547	Seatcushion Easy-Adapt SB38 x ST46cm black with red seam [2142+2153]		1
42	SP1529548	Seatcushion Easy-Adapt SB38 x ST51cm black with red seam [2142+2153]		1
43	SP1529549	Seatcushion Easy-Adapt SB43 x ST41cm black with red seam [2142+2153]		1
44	SP1529550	Seatcushion Easy-Adapt SB43 x ST46cm black with red seam [2142+2153]		1
45	SP1529551	Seatcushion Easy-Adapt SB43 x ST51cm black with red seam [2142+2153]		1
46	SP1529552	Seatcushion Easy-Adapt SB48 x ST41cm black with red seam [2142+2153]		1
47	SP1529553	Seatcushion Easy-Adapt SB48 x ST46cm black with red seam [2142+2153]		1
48	SP1529554	Seatcushion Easy-Adapt SB48 x ST51cm black with red seam [2142+2153]		1
49	SP1529555	Seatcushion Easy-Adapt SB53 x ST43cm black with red seam [2142+2153]		1
50	SP1529556	Seatcushion Easy-Adapt SB53 x ST46cm black with red seam [2142+2153]		1

Seat Cushions "Standard", "Matrx Flo-Tech" & "Matrx Vi"

Pos.	Part number	Description	Valid from → until	Qty
51	SP1529570	Seatcushion Easy-Adapt SB53 x ST51cm black with red seam [2142+2153]		1
52	SP1531760	Seat Cushion Matrx Vi SW 40 x SD 45cm (16" x 18") [2144]		1
53	SP1531761	Seat Cushion Matrx Vi SW 45 x SD 45cm (18" x 18") [2144]		1
54	SP1501113	Seat Cushion Flo-tech Contur Visco 43x43cm [2143]		1

Seat Cushions "Matrx Libra"

1803356-000

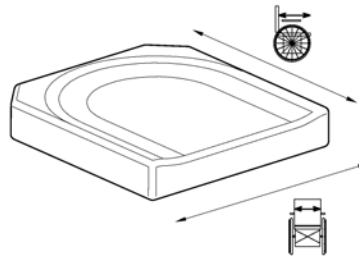


Matrx Libra

		mm			
		380	430	480	530
mm	410	1	4	7	-
	450	2	5	8	10
	510	3	6	9	11
	560	-	-	-	12

Pos.	Part number	Description	Valid from → until	Qty
1	LC1516	Matrx Libra cushions 380x410 mm		1
2	LC1518	Matrx Libra cushions 380x460 mm		1
3	LC1520	Matrx Libra cushions 380x510 mm		1
4	LC1716	Matrx Libra cushions 430x410 mm		1
5	LC1718	Matrx Libra cushions 430x460 mm		1
6	LC1720	Matrx Libra cushions 430x510 mm		1
7	LC1916	Matrx Libra cushions 480x410 mm		1
8	LC1918	Matrx Libra cushions 480x460 mm		1
9	LC1920	Matrx Libra cushions 480x510 mm		1
10	LC2118	Seat Cushion Matrx Libra 21x18"		1
11	LC2120	Seat Cushion Matrx Libra 21x20"		1
12	LC2122	Seat Cushion Matrx Libra 21x22"		1

Backrest

SFCFDX25.060

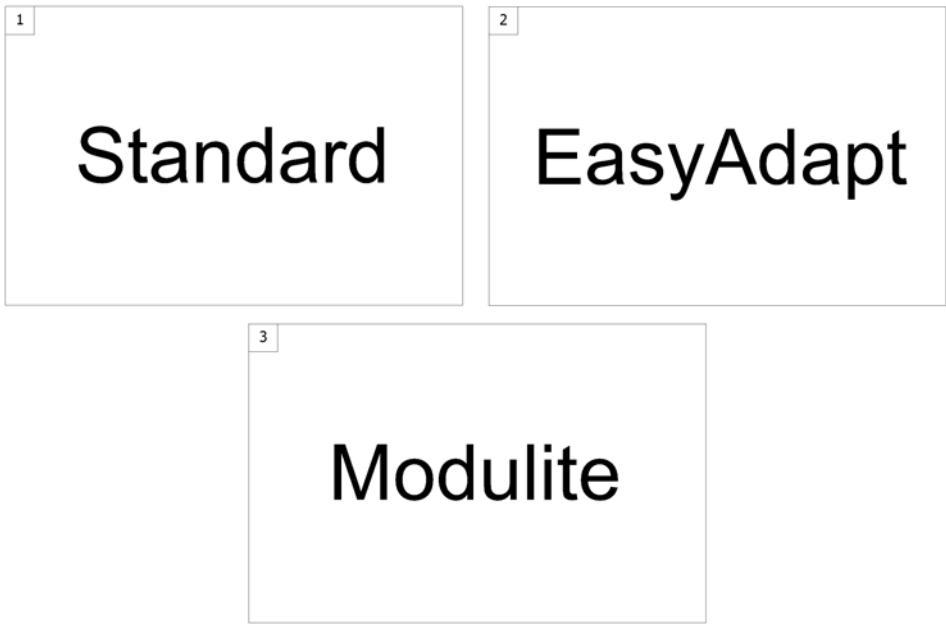


Pos.	Part number	Description	Valid from → until	Qty
1	-	Backrest for Standard Seat		1
2	-	Backrest for EasyAdapt Seat		1
3	-	Backrest for Modulite Seat		1



Pos.	Part number	Description	Valid from → until	Qty
1	-	Standard Backrest [1178/0970]		1

Standard Backrest


1526951X.dwg

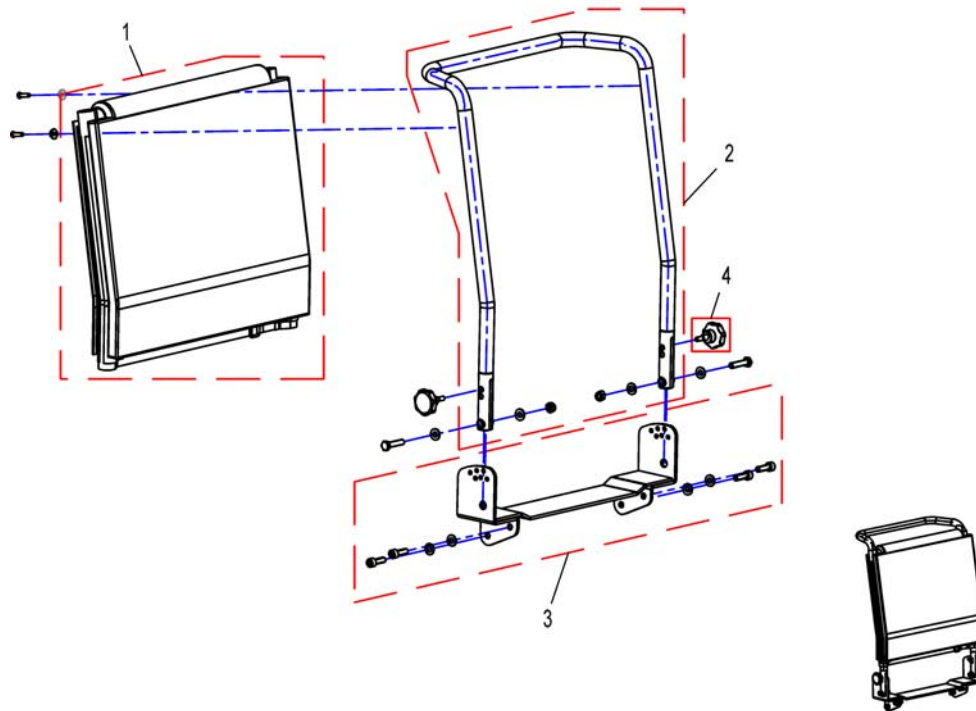

Pos.	Part number	Description	Valid from → until	Qty
1	1422062	Sling Back with Backrest upholstery 480 mm		1
1	1422063	Sling Back with Backrest upholstery 540mm		1
2	SP1419996	Backrest Frame 48cm height		1
2	SP1420465	Backrest Frame 54cm height		1
3	SP1530528	Bracket for standard backrest		1
4	SP1420444	Star Grip M8		1


Backrest for EasyAdapt Seat

1573229 iso

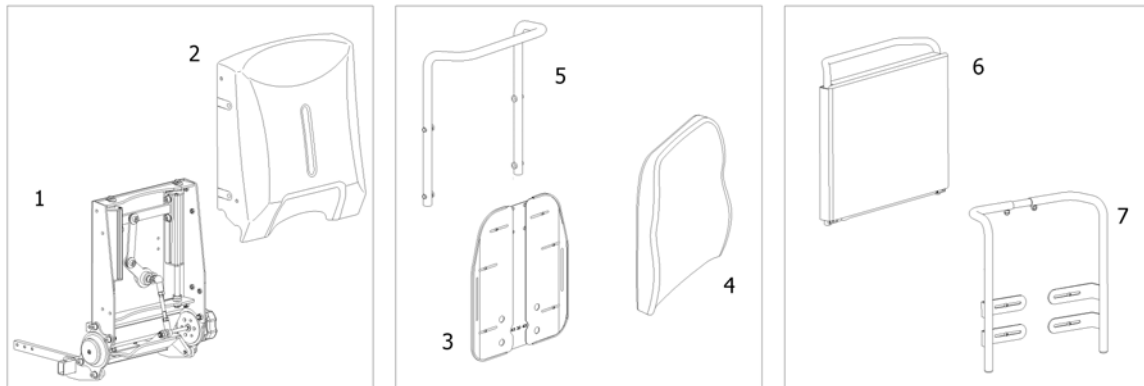


Pos.	Part number	Description	Valid from → until	Qty
1	-	Backrest adjustment		1
2	-	Backrestcover for EasyAdapt		1
3	-	Backplate EasyAdapt		1
3	-	Backplate EasyAdapt for DAHL Docking Station [1450]		1
4	-	Backrest Cushion [1985/1986/1987/1988]		1
5	-	Push Handle EasyAdapt [1307]		1
6	-	Tension adjustable Sling Back [1185/1186]		1
7	-	"Universal"-Backrest frame [0904]		1

Backrest adjustment

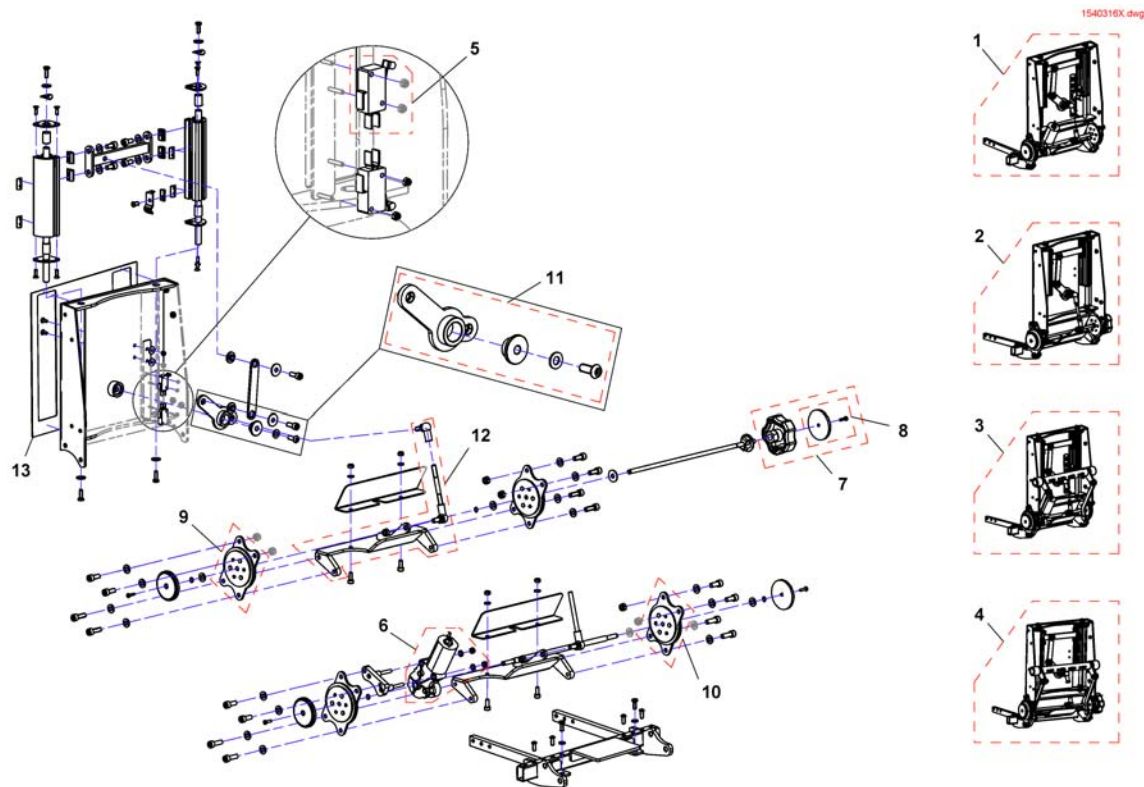


Pos.	Part number	Description	Valid from → until	Qty
-		Following armrests		1
1	SP1533413	Electrical Backrest, cpl.		1
2	SP1540316	Manuell Backrest, adjustment RH, cpl.		1
2	SP1540317	Manuell Backrest, adjustment LH, cpl.		1
3	SP1540321	Electrical Backrest, following Armrest, cpl.		1
4	SP1540319	Manuell Backrest, following armrest, adjustment RH, cpl.		1
4	SP1540320	Manuell Backrest, following armrest, adjustment LH, cpl.		1
5	SP1526445	Cable loom for stop switch		1
6	SP1530520	Gearmotor		1
7	SP1420483	Hand Wheel, cpl.		1
8	SP1421696	Reflector cpl.		1
9	SP1450234	SEAT BRACE MANUAL BACKREST, 1 PAIR		1
10	SP1423211	SEAT BRACE ELECTRIC BACKREST, 1 PAIR		1
11	SP1535255	Retrofitting-Kit Backrest		1
12	SP1537863	Bridge for shearcompensation		1
13	SP1532897	Backrest sliding plate		1

Following armrests

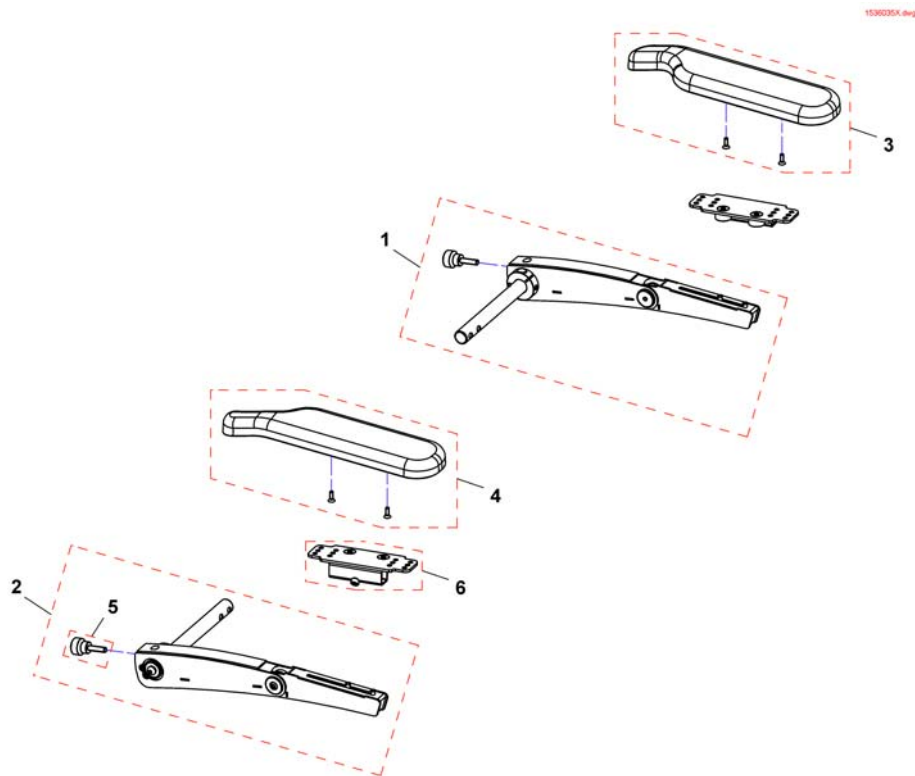

Pos.	Part number	Description	Valid from → until	Qty
1	SP1536036	Following armrest, short, LH, cpl.		1
1	SP1536100	Following armrest, long, LH, cpl.		1
2	SP1536035	Following armrest, short, RH, cpl.		1
2	SP1536099	Following armrest, long, RH, cpl.		1
3	SP1443004	Armpad, Standard (325mm), LH		1
3	SP1528846	Armpad, Extended (375mm), LH		1
4	SP1449043	Armpad, Standard (325mm), RH		1
4	SP1528845	Armpad, Extended (375mm), RH		1
5	SP1535504	Knurled screw M8x30		1
6	SP1535244	Armpad holder		1


Backrestcover for EasyAdapt

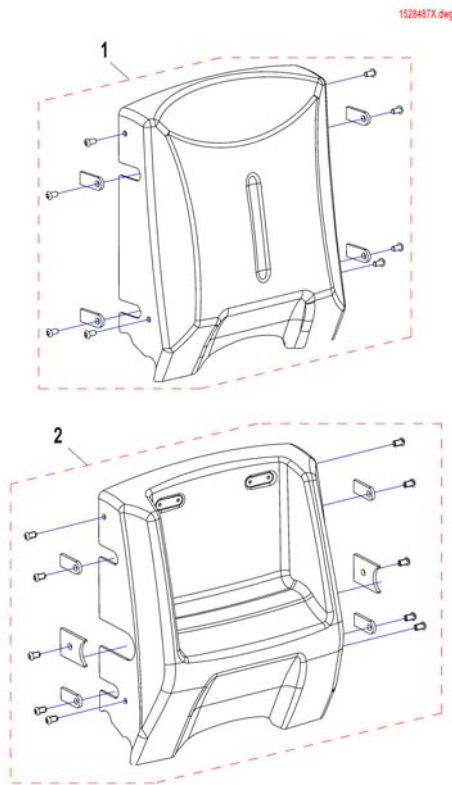

Pos.	Part number	Description	Valid from → until	Qty
1	SP1528487	Backrestcover for EasyAdpt		1
2	SP1537706	Backrestcover EasyAdapt for following armrest		1

Backplate EasyAdapt


1528949X.dwg

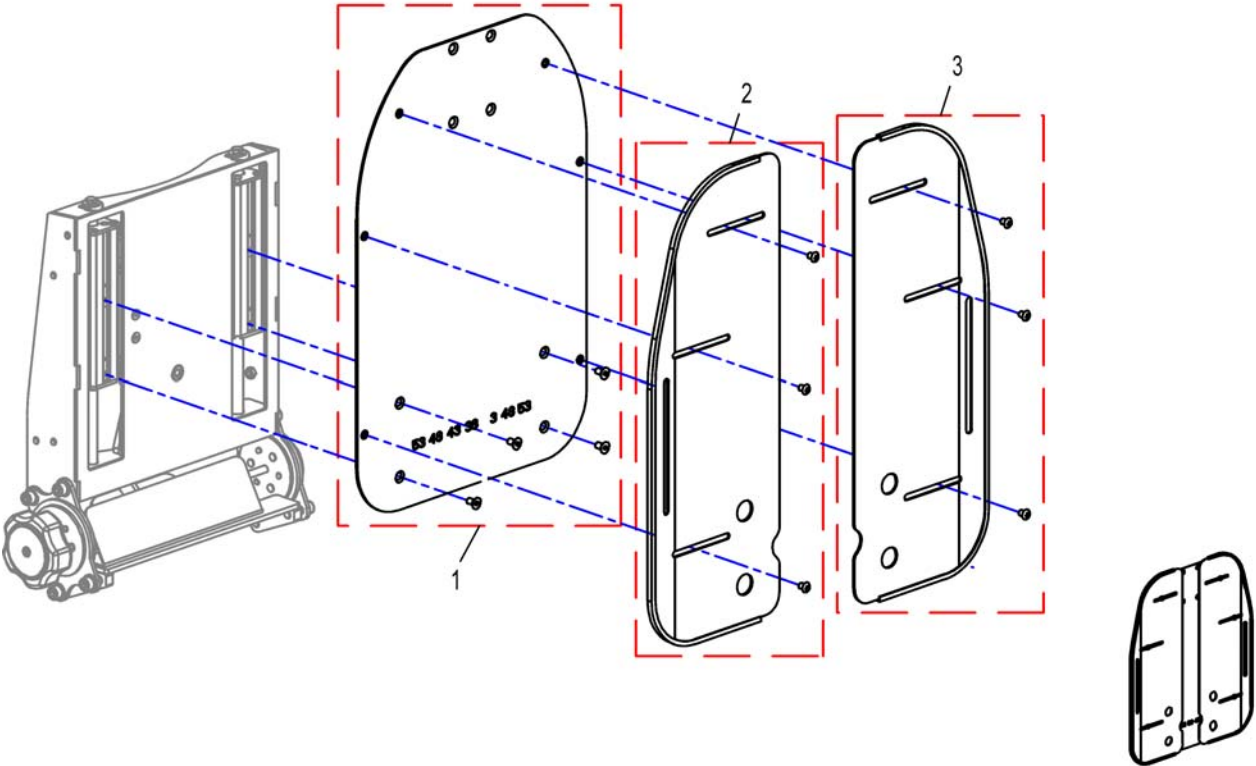


Pos.	Part number	Description	Valid from → until	Qty
1	SP1530525	Backplate EasyAdapt, middle		1
2	SP1530526	Backplate EasyAdapt, right		1
3	SP1530527	Backplate EasyAdapt, left		1

Backplate EasyAdapt for DAHL Docking Station

1579892 iso

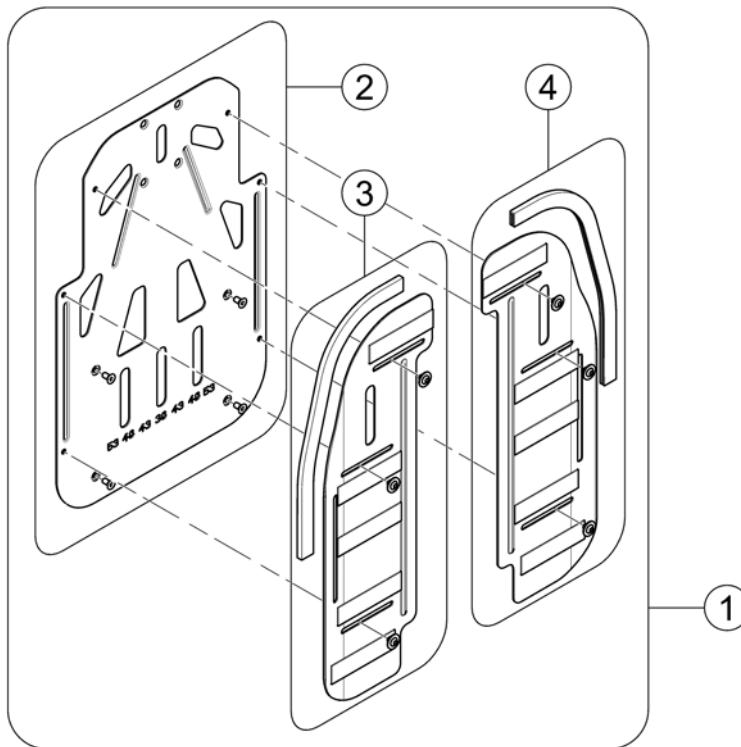


Pos.	Part number	Description	Valid from → until	Qty
1	SP1541384	Backrest Plate Assy, reinforced - for DAHL Docking Station		1
2	SP1579843	Center Backrest Plate, reinforced - for DAHL Docking Station		1
3	SP1579844	Back Plate left, reinforced - for DAHL Docking Station		1
4	SP1579845	Back Plate right, reinforced - for DAHL Docking Station		1

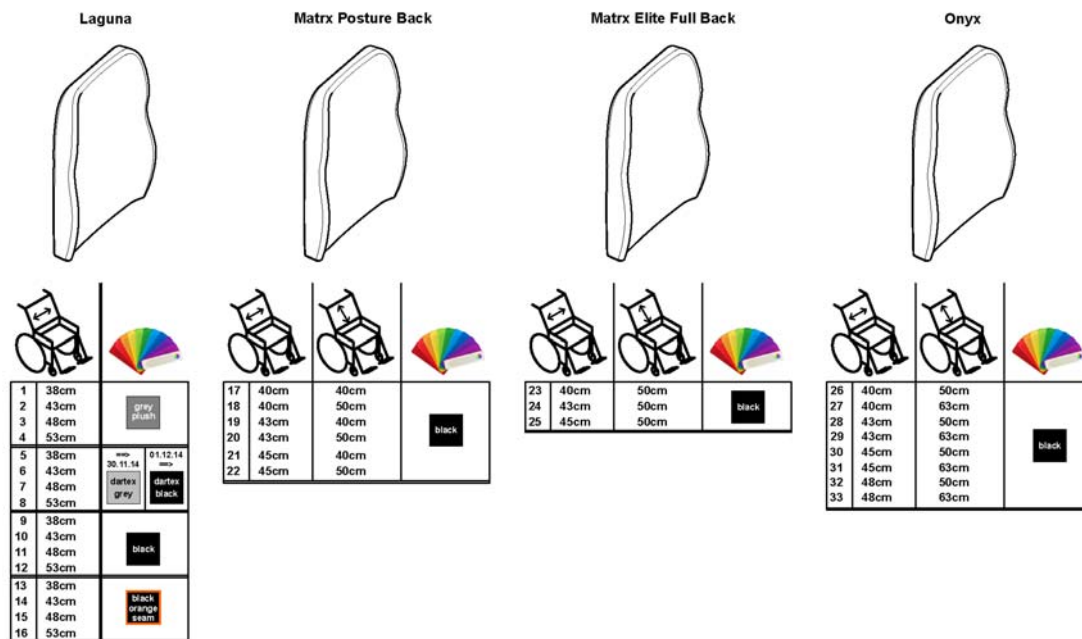

Pos.	Part number	Description	Valid from → until	Qty
1	1444339	Backrest Cushion Laguna, 372x550mm, Grey Plush		1
2	1435798	Backrest Cushion Laguna, 402x550mm, Grey Plush		1
3	1444340	Backrest Cushion Laguna, 452x550mm, Grey Plush		1
4	1444341	Backrest Cushion Laguna, 492x550mm, Grey Plush		1
5	1427118	Backrest cushion comfort Laguna (XS), 372x550 mm, Dartex grey (TR23)	→ 2014-11-30	1
5	SP1529538-TR26	Backrest Cushion Laguna / EasyAdapt, Dartex black - SW 380mm	2014-12-01 →	1
6	-	No longer available - please choose Dartex black	→ 2014-11-30	1
6	SP1529539-TR26	Backrest Cushion Laguna / EasyAdapt, Dartex black - SW 430mm	2014-12-01 →	1
7	-	No longer available - please choose Dartex black	→ 2014-11-30	1
7	SP1529540-TR26	Backrest Cushion Laguna / EasyAdapt, Dartex black - SW 480mm	2014-12-01 →	1
8	1427121	Backrest Cushion Laguna, 492x550mm, Dartex Grey	→ 2014-11-30	1
8	SP1529545-TR26	Backrest Cushion Laguna / EasyAdapt, Dartex black - SW 530mm	2014-12-01 →	1
9	SP1529585	Backrest Cushion Laguna / EasyAdapt, black - SW 380mm		1
10	SP1529586	Backrest Cushion Laguna / EasyAdapt, black - SW 430mm		1
11	SP1529587	Backrest Cushion Laguna / EasyAdapt, black - SW 480mm		1
12	SP1529596	Backrest Cushion Laguna / EasyAdapt, black - SW 530mm		1
13	SP1529538	Backrest Cushion Laguna / EasyAdapt, black with orange seam - SW 380mm		1
14	SP1529539	Backrest Cushion Laguna / EasyAdapt, black with orange seam - SW 430mm		1
15	SP1529540	Backrest Cushion Laguna / EasyAdapt, black with orange seam - SW 480mm		1
16	SP1529545	Backrest Cushion Laguna / EasyAdapt, black with orange seam - SW 530mm		1
17	SP1531762	Backrest, Matrix Posture Back 16x16", cpl. w/ mounting hardware		1
18	SP1531763	Backrest, Matrix Posture Back 16x20", cpl. w/ mounting hardware		1
19	SP1531792	Backrest, Matrix Posture Back 17x16", cpl. w/ mounting hardware		1
20	SP1531793	Backrest, Matrix Posture Back 17x20", cpl. w/ mounting hardware		1
21	SP1531822	Backrest, Matrix Posture Back 18x16", cpl. w/ mounting hardware		1
22	SP1531823	Backrest, Matrix Posture Back 18x20", cpl. w/ mounting hardware		1
23	SP1531824	Backrest, Matrix Elite (Full) Back 16x20", cpl. w/ mounting hardware		1
24	SP1531831	Backrest, Matrix Elite (Full) Back 17x20", cpl. w/ mounting hardware		1

Backrest Cushion


Pos.	Part number	Description	Valid from → until	Qty
25	SP1531832	Backrest, Matrix Elite (Full) Back 18x20", cpl. w/ mounting hardware		1
26	SP1532061	ONYX Back 40x50cm (16" x 20")		1
27	SP1532062	ONYX Back 40x63cm (16" x 25")		1
28	SP1532064	ONYX Back 43x50cm (17" x 20")		1
29	SP1532065	ONYX Back 43x63cm (17" x 25")		1
30	SP1532118	ONYX Back 45x50cm (18" x 20")		1
31	SP1532194	ONYX Back 45x63cm (18" x 25")		1
32	SP1532211	ONYX Back 48x50cm (19" x 20")		1
33	SP1532212	ONYX Back 48x63cm (19" x 25")		1

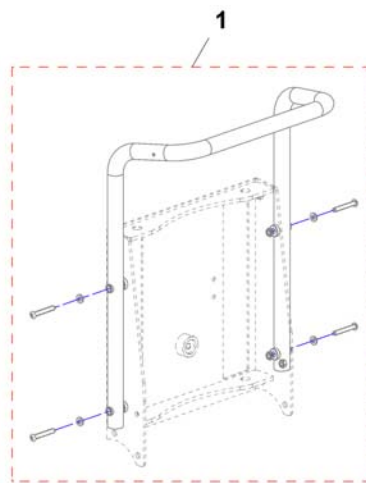

Pos.	Part number	Description	Valid from → until	Qty
1	SP1529001	Push Handle EasyAdapt-Seat		1

Tension adjustable Sling Back


1533409X.dwg

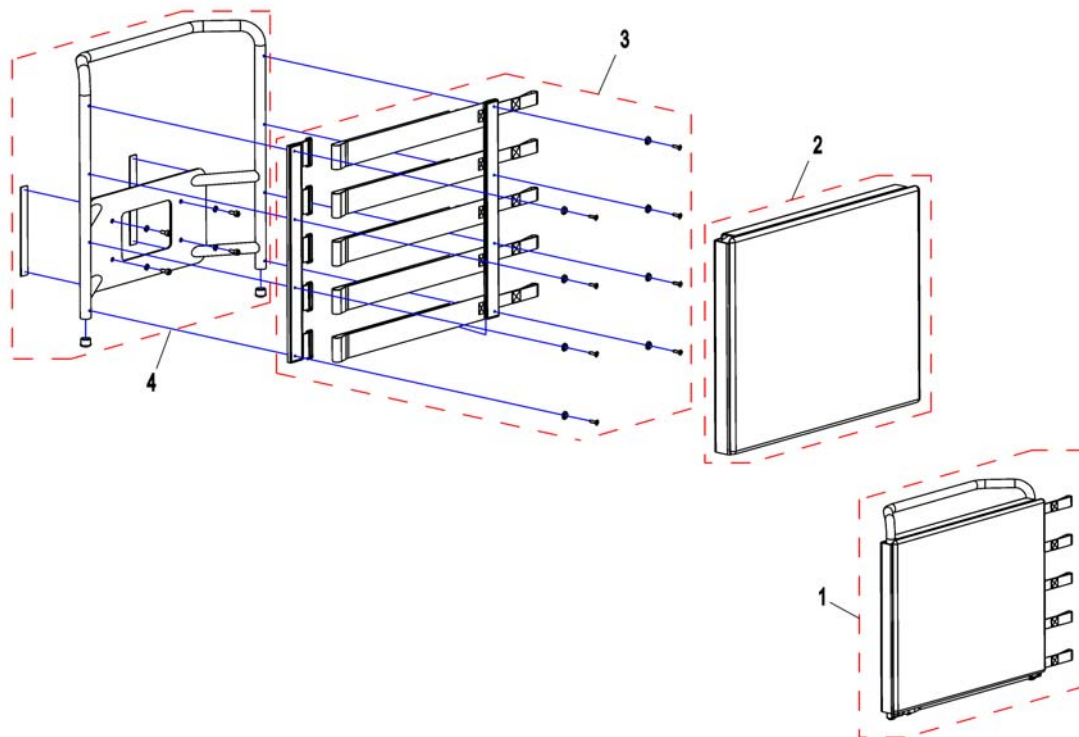

Pos.	Part number	Description	Valid from → until	Qty
1	SP1531825	Lashing Backrest SW 43, cpl.		1
1	SP1533409	Lashing Backrest SW 53, cpl.		1
2	SP1531830	Backrestupholstery SW 43		1
2	SP1533405	Backrestupholstery SW 53		1
3	SP1537258	Lashingstrap SW 43, cpl.		1
3	SP1537296	Lashingstrap SW 53, cpl.		1
4	SP1537297	Backrestcarrier SW 43, cpl.		1
4	SP1537298	Backrestcarrier SW 53, cpl.		1

"Universal"-Backrest frame


1531052X.dwg

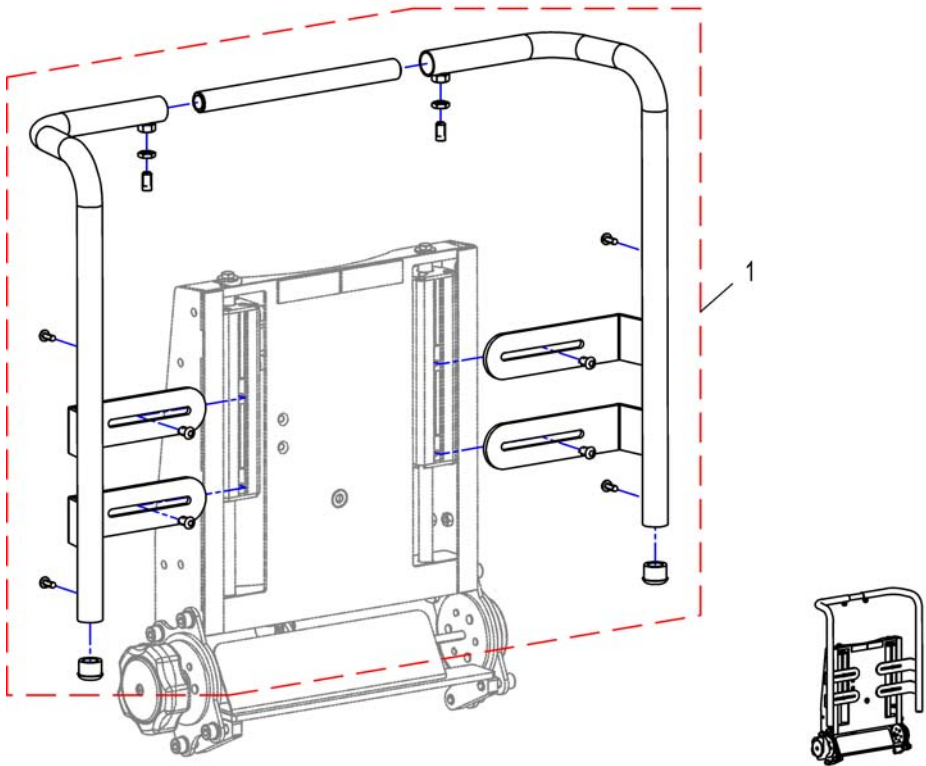

Pos.	Part number	Description	Valid from → until	Qty
1	SP1531052	"Universal"-Backrest frame		1

Backrest for Modulite Seat


1586095.kso

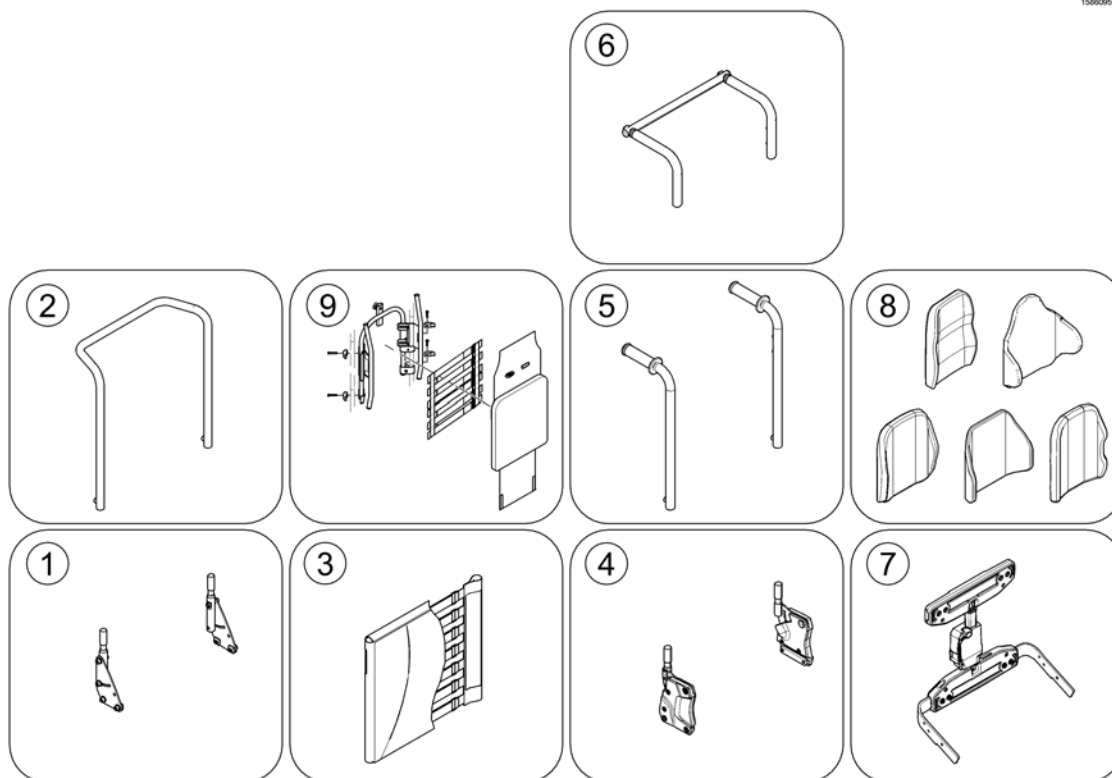

Pos.	Part number	Description	Valid from → until	Qty
1	-	Support for Backrest Frame [0975+2161]		1
2	-	Backrest Frame [2161]		1
3	-	Sling Back, tension adjustable [2171]		1
4	-	Backrest support		1
5	-	Backposts [2162/2163]		1
6	-	Connection tube [1307]		1
7	-	Back Recliner / Stabilization bracket [0977]		1
8	-	Backrest Cushions [2172/2173/2174/2175/2176/2177/2178]		1
9	-	Flex3 back, tension adjustable [2179]		1


Support for Backrest Frame

1571553.iso

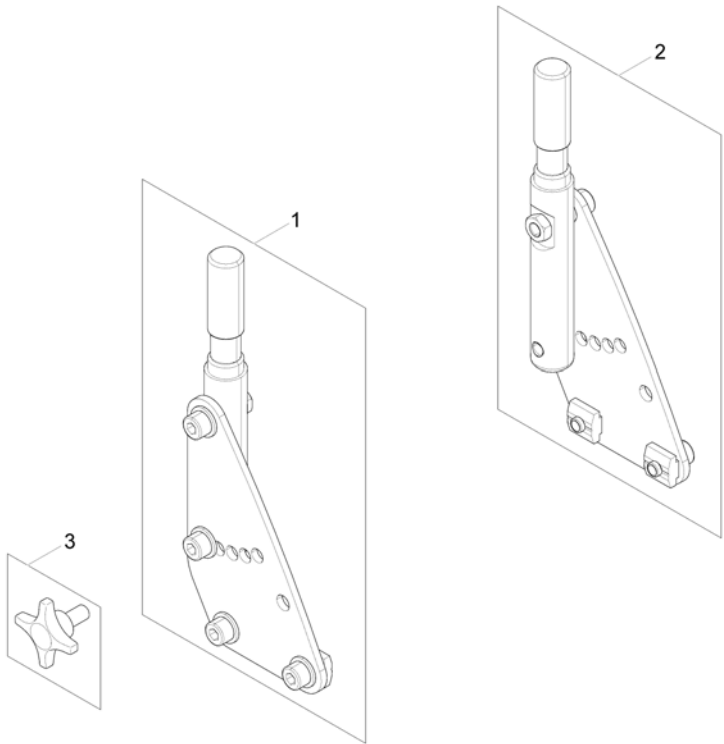

Pos.	Part number	Description	Valid from → until	Qty
1	SP1571542	Backrest Support RH		1
2	SP1571553	Backrest Support LH		1
3	5824117	hand wheel M8x15 <i>(only swedish homologation)</i>		2


Backrest Frame

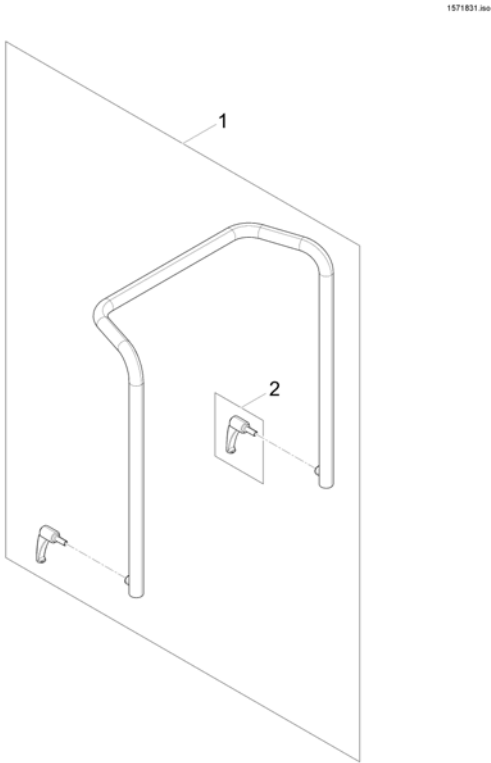

Pos.	Part number	Description	Valid from → until	Qty
1	SP1567810	Backrest Frame		1
2	0021-70002	Indexable lever 63mm M8x16		1

Sling Back, tension adjustable


1571829.indd

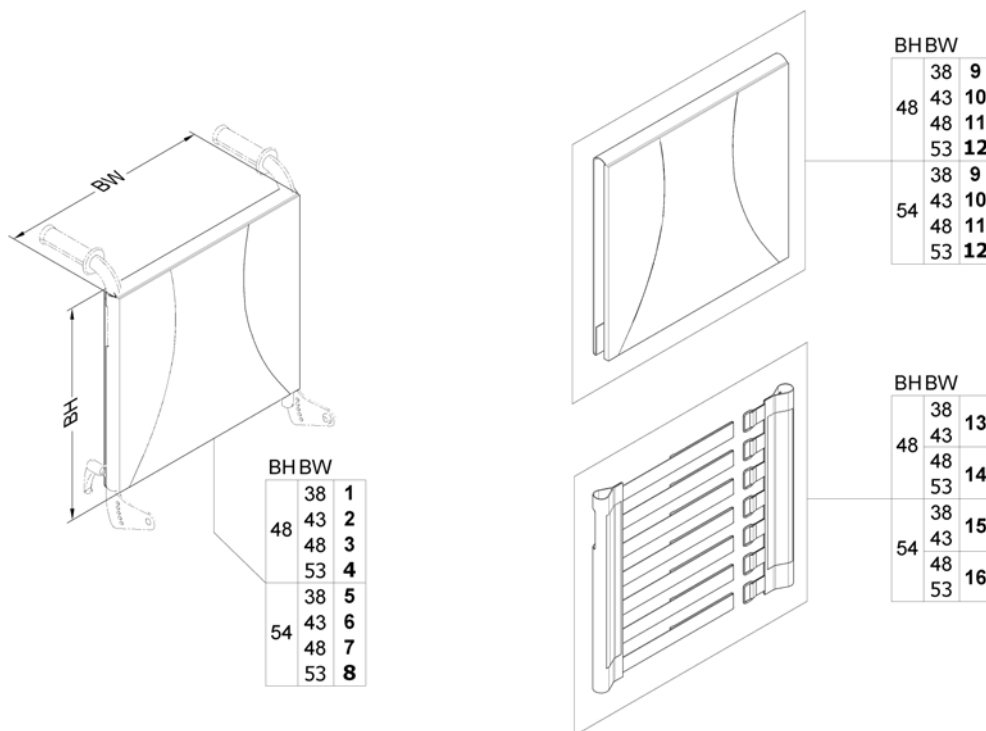

Pos.	Part number	Description	Valid from → until	Qty
1	SP1567529	Sling Back, tension adjustable & Upholstery, BW 380mm / BH 480mm, cpl.		1
2	SP1567465	Sling Back, tension adjustable & upholstery, BW 430mm / BH 480mm, cpl.		1
3	SP1567530	Sling Back, tension adjustable & upholstery, BW 480mm / BH 480mm, cpl.		1
4	SP1567531	Sling Back, tension adjustable & upholstery, BW 530mm / BH 480mm, cpl.		1
5	SP1567627	Sling Back, tension adjustable & upholstery, BW 380mm / BH 540mm, cpl.		1
6	SP1567746	Sling Back, tension adjustable & upholstery, BW 430mm / BH 540mm, cpl.		1
7	SP1567747	Sling Back, tension adjustable & upholstery, BW 480mm / BH 540mm, cpl.		1
8	SP1567748	Sling Back, tension adjustable & upholstery, BW 530mm / BH 540mm, cpl.		1
9	SP1563563	Backrest upholstery		1
10	SP1563653	Backrest upholstery		1
11	SP1563654	Backrest upholstery		1
12	SP1563655	Backrest upholstery		1
13	SP1571492	Sling back		1
14	SP1571493	Sling back		1
15	SP1571494	Sling back		1
16	SP1571510	Sling back		1


Backrest support

1572928-00

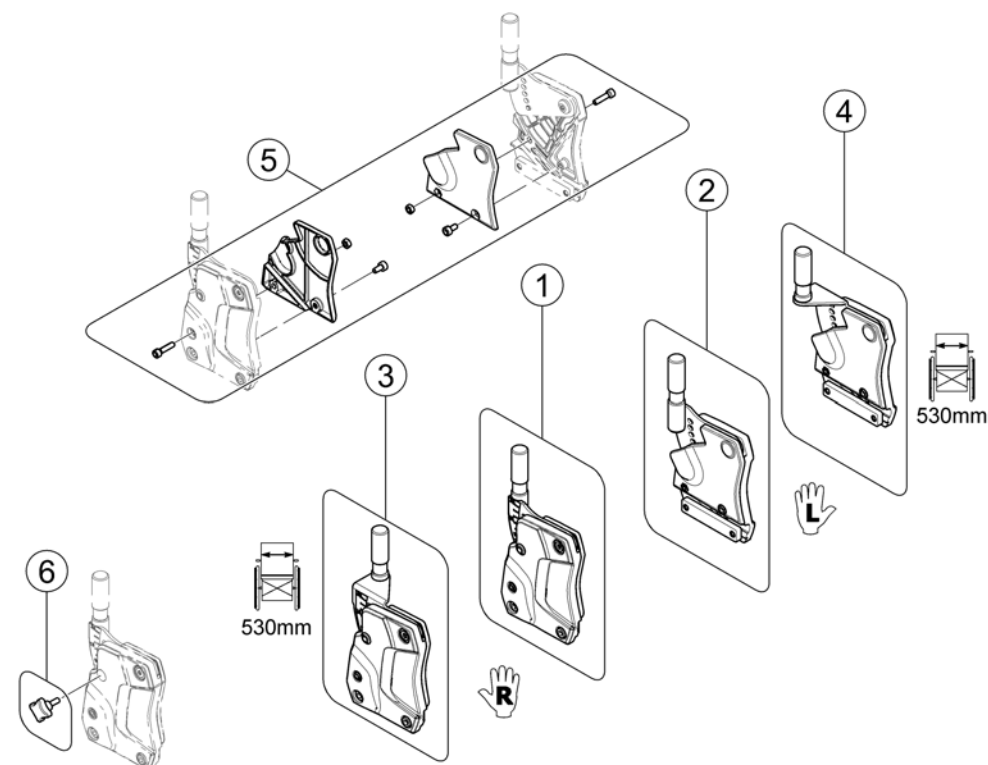

Pos.	Part number	Description	Valid from → until	Qty
1	SP1582709	Backrest Support - RH <i>Compatible with Flip-Up & Following Armrest [2193/2192/2194] - replaces SP1571560</i>		1
2	SP1582708	Backrest Support - LH <i>Compatible with Flip-Up & Following Armrest [2193/2192/2194] - replaces SP1571561</i>		1
3	SP1582952	Backrest Support SW 530mm - RH <i>Compatible with Flip-Up & Following Armrest [2193/2192/2194] - replaces SP1571568</i>		1
4	SP1582951	Backrest Support SW 530mm - LH <i>Compatible with Flip-Up & Following Armrest [2193/2192/2194] - replaces SP1571569</i>		1
5	SP1571676	Cover		1
6	SP1579877	Hand Screw M6 <i>(Swedish Homologation only!)</i>	→ 2015-07-31	2
6	SP1589324	Cross handle M8, cpl. <i>(Swedish Homologation only!)</i>	2015-08-01 →	2


Backposts

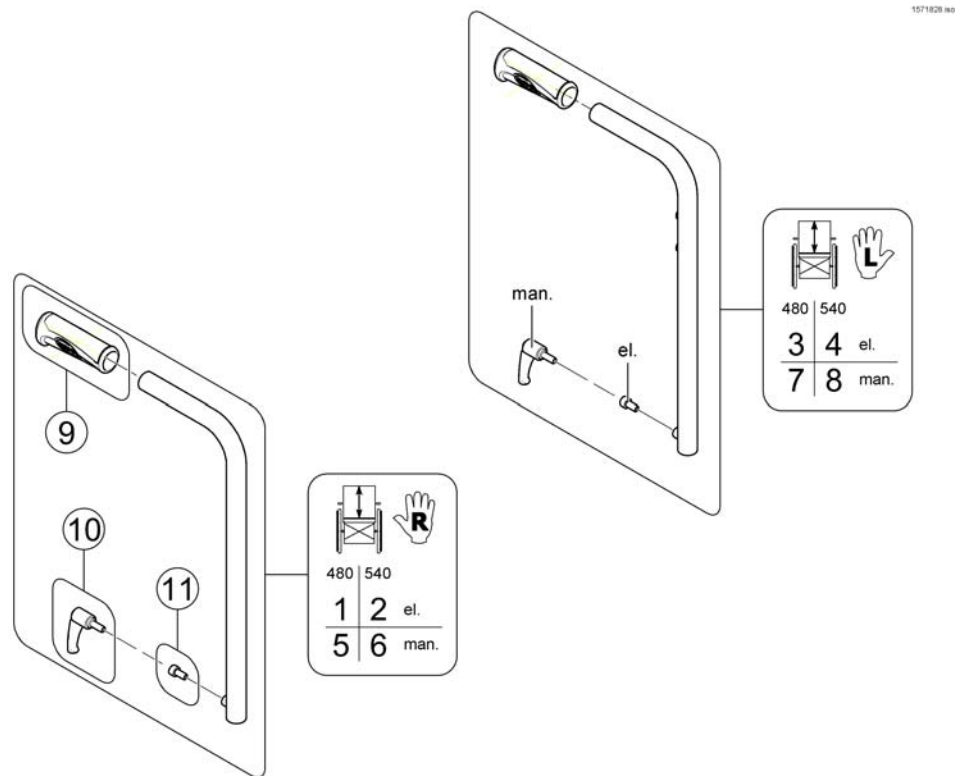

Pos.	Part number	Description	Valid from → until	Qty
1	SP1587177	Backpost el. BH 480mm - RH <i>Compatible with Flip-Up & Following Armrest [2193/2192/2194] - replaces SP1563102</i>		1
2	SP1587184	Backpost el. BH 540mm - RH <i>Compatible with Flip-Up & Following Armrest [2193/2192/2194] - replaces SP1563103</i>		1
3	SP1587176	Backpost el. BH 480mm - LH <i>Compatible with Flip-Up & Following Armrest [2193/2192/2194] - replaces SP1560576</i>		1
4	SP1587183	Backpost el. BH 540mm - LH <i>Compatible with Flip-Up & Following Armrest [2193/2192/2194] - replaces SP1559578</i>		1
5	SP1587163	Backpost man. BH 480mm - RH <i>Compatible with Flip-Up & Following Armrest [2193/2192/2194] - replaces SP1572489</i>		1
6	SP1587182	Backpost man. BH 540mm - RH <i>Compatible with Flip-Up & Following Armrest [2193/2192/2194] - replaces SP1572490</i>		1
7	SP1587162	Backpost man. BH 480mm - LH <i>Compatible with Flip-Up & Following Armrest [2193/2192/2194] - replaces SP1572491</i>		1
8	SP1587178	Backpost man. BH 540mm - LH <i>Compatible with Flip-Up & Following Armrest [2193/2192/2194] - replaces SP1572492</i>		1
9	SP1586122	Handles (pair) <i>replaces item SP1563651</i>		1
10	0021-70002	Indexable lever 63mm M8x16		1
11	1567999	Bolt M8x16 8.8 black DIN912		1

Connection tube


1571835 iso

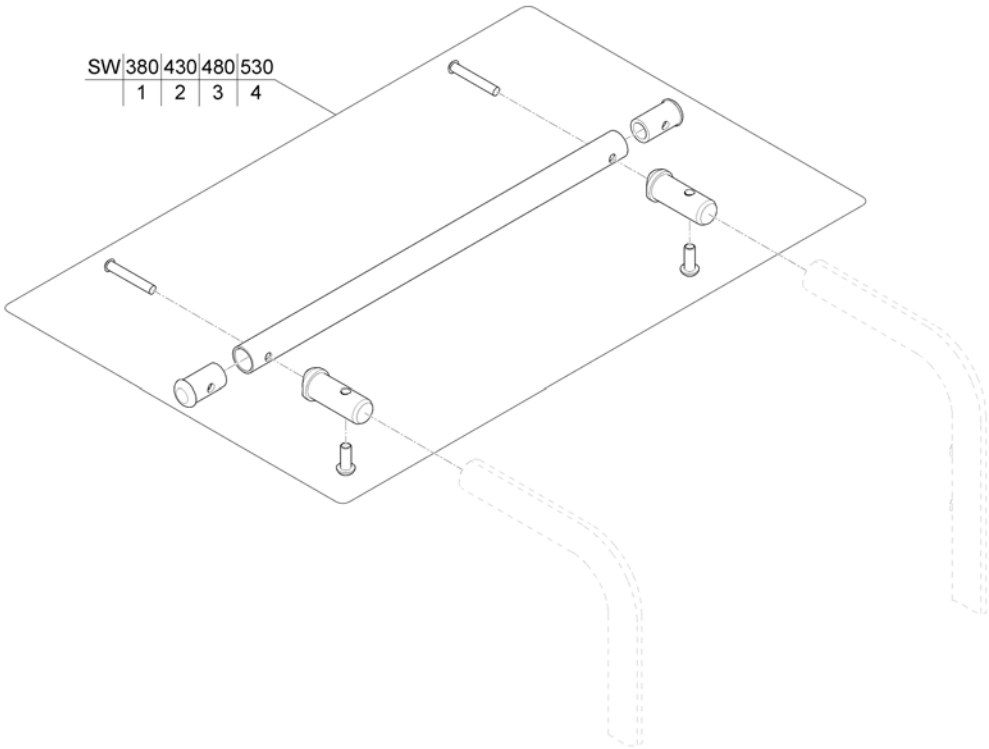

Pos.	Part number	Description	Valid from → until	Qty
1	SP1570370	Connection tube		1
2	SP1570371	Connection tube		1
3	SP1570372	Connection tube		1
4	SP1570373	Connection tube		1

Back Recliner / Stabilization bracket


1571697.doc

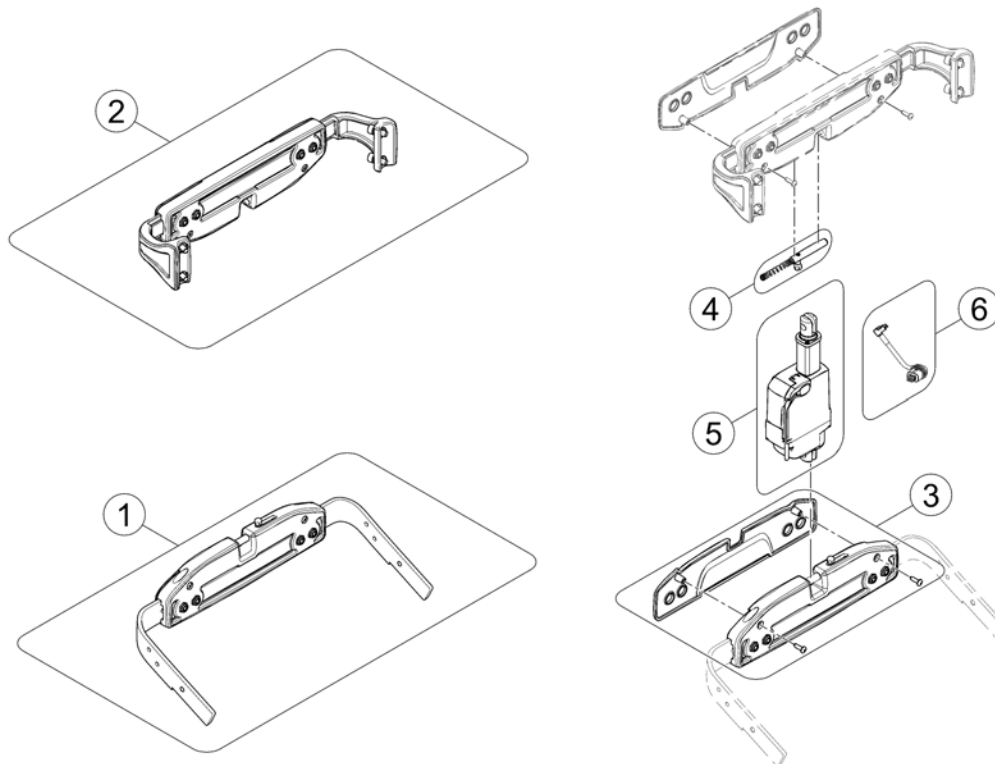

Pos.	Part number	Description	Valid from → until	Qty
1	SP1567169	Actuator bracket (pre set to SW 43cm)		1
2	SP1567168	Stabilization bracket (pre set to SW 43cm)		1
3	SP1571702	Cover		1
4	SP1571704	Locking bolt		1
5	SP1557820	Actuator		1
6	SP1572166	Actuator Cable LA23 ACS2 (~950mm)		1

Backrest Cushions



1508915.000

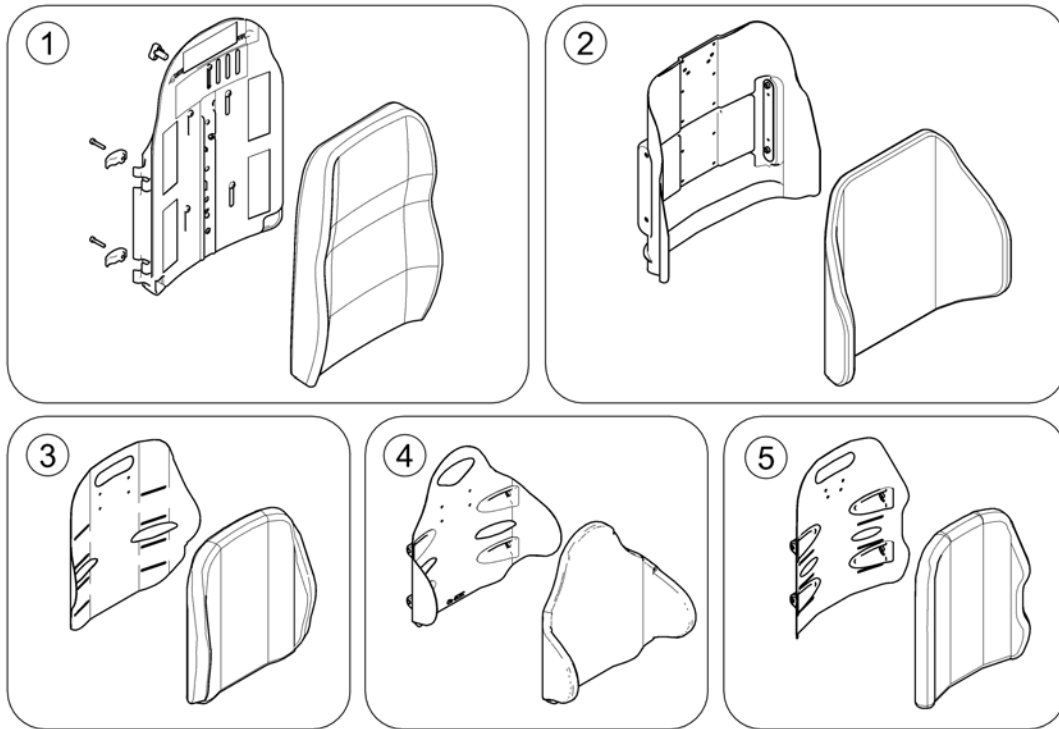

Pos.	Part number	Description	Valid from → until	Qty
1	-	Flex 3 / Laguna		1
2	-	Matrx Posture Back		1
3	-	Matrx Elite T/R Back	2015-07-01 →	1
4	-	Matrx Elite Deep Back	2015-07-01 →	1
5	-	Matrx Elite (Full) Back		1

Flex 3 / Laguna

1573106.000

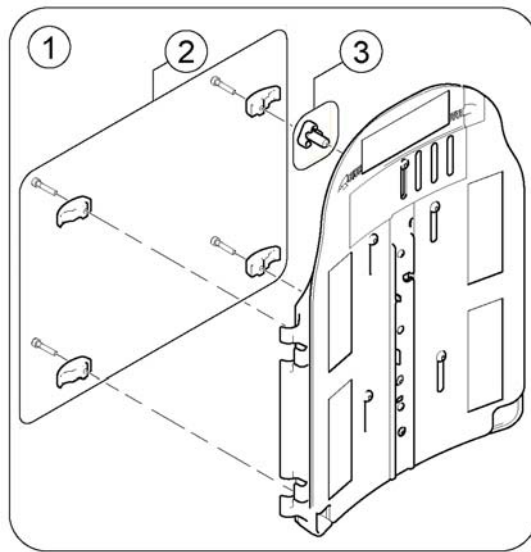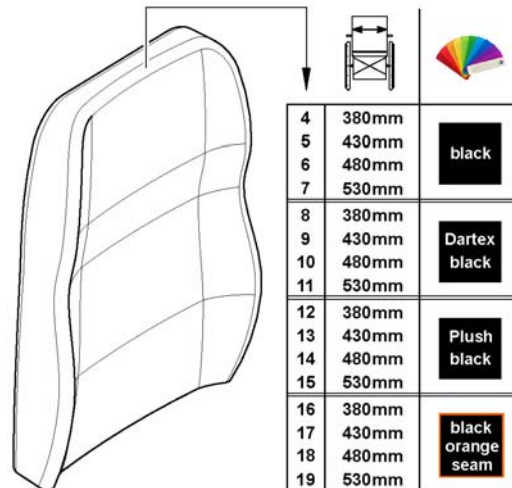**Flex 3****Laguna**


Pos.	Part number	Description	Valid from → until	Qty
1	SP1568489	Flex3 backrest plate [2172]		1
2	SP1572321	Clamps		1
3	1497411	HANDWHEEL M12 LONG NECK/HEADREST		1
4	SP1529585	Backrest Cushion Laguna / EasyAdapt, black - SW 380mm [2173+2151]		1
5	SP1529586	Backrest Cushion Laguna / EasyAdapt, black - SW 430mm [2173+2151]		1
6	SP1529587	Backrest Cushion Laguna / EasyAdapt, black - SW 480mm [2173+2151]		1
7	SP1529596	Backrest Cushion Laguna / EasyAdapt, black - SW 530mm [2173+2151]		1
8	SP1529538-TR26	Backrest Cushion Laguna / EasyAdapt, Dartex black - SW 380mm [2173+2152]		1
9	SP1529539-TR26	Backrest Cushion Laguna / EasyAdapt, Dartex black - SW 430mm [2173+2152]		1
10	SP1529540-TR26	Backrest Cushion Laguna / EasyAdapt, Dartex black - SW 480mm [2173+2152]		1
11	SP1529545-TR26	Backrest Cushion Laguna / EasyAdapt, Dartex black - SW 530mm [2173+2152]		1
12	SP1529538-TR35	Backrest Cushion Laguna, Plush black - SW 380mm [2173+2154]		1
13	SP1529539-TR35	Backrest Cushion Laguna, Plush black - SW 430mm [2173+2154]		1
14	SP1529540-TR35	Backrest Cushion Laguna, Plush black - SW 480mm [2173+2154]		1
15	SP1529545-TR35	Backrest Cushion Laguna, Plush black - SW 530mm [2173+2154]		1
16	SP1529538	Backrest Cushion Laguna / EasyAdapt, black with orange seam - SW 380mm [2173+2153]		1
17	SP1529539	Backrest Cushion Laguna / EasyAdapt, black with orange seam - SW 430mm [2173+2153]		1
18	SP1529540	Backrest Cushion Laguna / EasyAdapt, black with orange seam - SW 480mm [2173+2153]		1

Flex 3 / Laguna

Pos.	Part number	Description	Valid from → until	Qty
19	SP1529545	Backrest Cushion Laguna / EasyAdapt, black with orange seam - SW 530mm [2173+2153]		1

Matrx Posture Back


158885 Rev

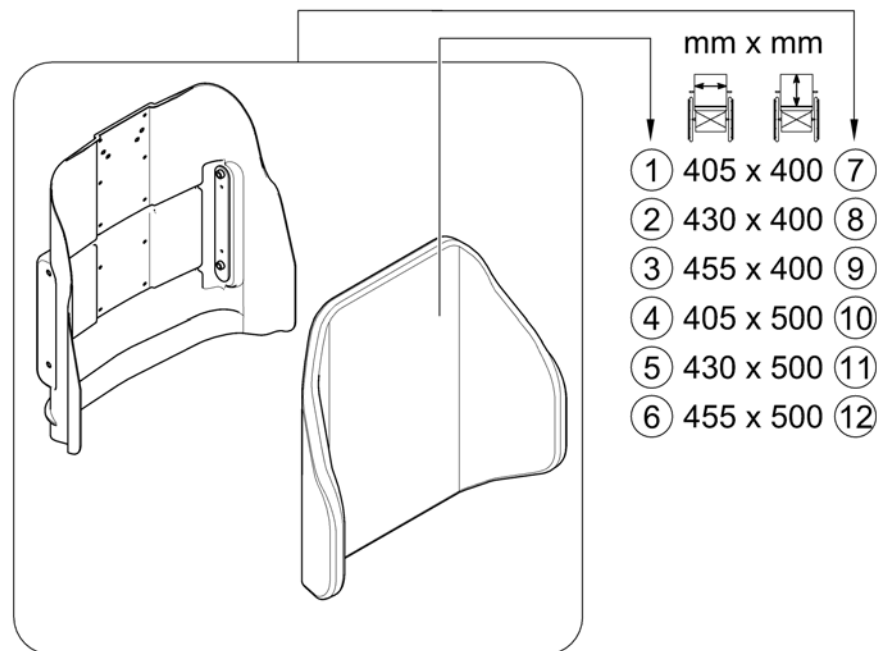Posture Back

Pos.	Part number	Description	Valid from → until	Qty
1	1544896	Backrest Cushion - Posture Back 405x400mm (16x16")		1
2	1544899	Backrest Cushion - Posture Back 430x400mm (17x16")		1
3	1544995	Backrest Cushion - Posture Back 455x400mm (18x16")		1
4	1544897	Backrest Cushion - Posture Back 405x500mm (16x20")		1
5	1544982	Backrest Cushion - Posture Back 430x500mm (17x20")		1
6	1544996	Backrest Cushion - Posture Back 455x500mm (18x20")		1
7	SP1531762	Backrest, Matrx Posture Back 16x16", cpl. w/ mounting hardware [2174]		1
8	SP1531792	Backrest, Matrx Posture Back 17x16", cpl. w/ mounting hardware [2174]		1
9	SP1531822	Backrest, Matrx Posture Back 18x16", cpl. w/ mounting hardware [2174]		1
10	SP1531763	Backrest, Matrx Posture Back 16x20", cpl. w/ mounting hardware [2175]		1
11	SP1531793	Backrest, Matrx Posture Back 17x20", cpl. w/ mounting hardware [2175]		1
12	SP1531823	Backrest, Matrx Posture Back 18x20", cpl. w/ mounting hardware [2175]		1

Matrx Elite T/R Back


1588888 (n)

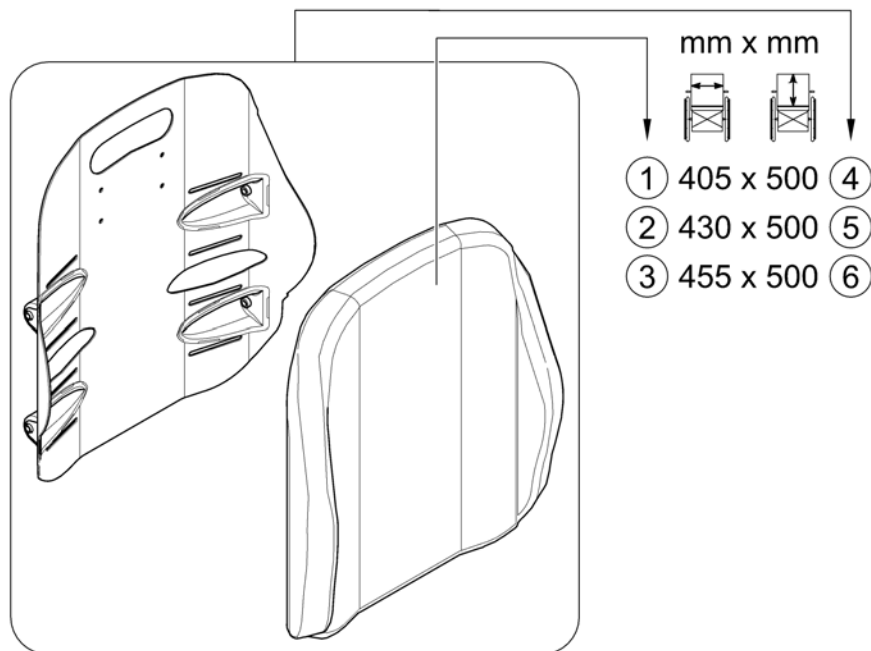Elite T/R Back


Pos.	Part number	Description	Valid from → until	Qty
1	1580110	Backrest Cushion - Elite T/R Back 405x500mm (16x20")		1
2	1580297	Backrest Cushion - Elite T/R Back 430x500mm (17x20")		1
3	1580298	Backrest Cushion - Elite T/R Back 455x500mm (18x20")		1
4	SP1588270	Backrest, Matrx Elite T/R Back 16x20", cpl. w/ mounting hardware [2177+2121]		1
5	SP1588271	Backrest, Matrx Elite T/R Back 17x20", cpl. w/ mounting hardware [2177+2122]		1
6	SP1588272	Backrest, Matrx Elite T/R Back 18x20", cpl. w/ mounting hardware [2177+2123]		1

Matrx Elite Deep Back

1588913.00

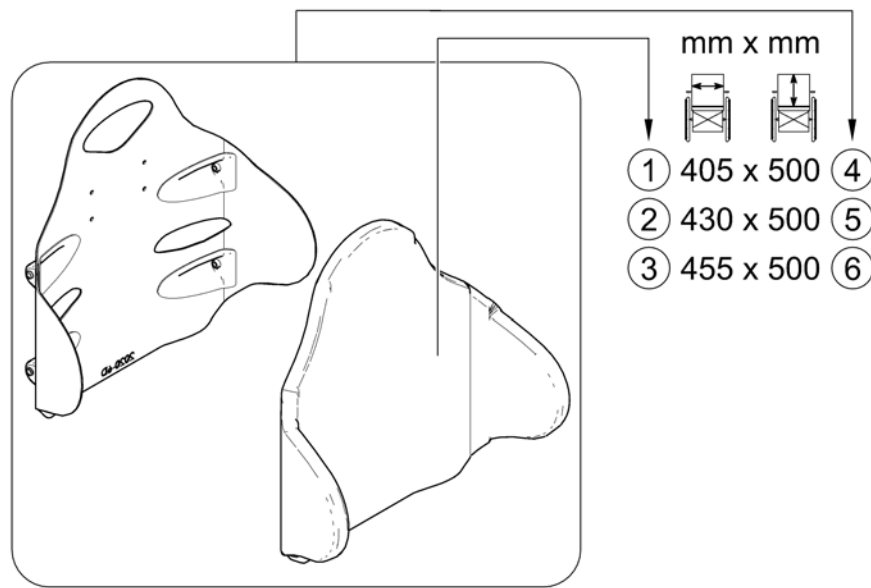Elite Deep Back


Pos.	Part number	Description	Valid from → until	Qty
1	1544073	Backrest Cushion - Elite Deep Back 405x500mm (16x20")		1
2	1544077	Backrest Cushion - Elite Deep Back 430x500mm (17x20")		1
3	1544081	Backrest Cushion - Elite Deep Back 455x500mm (18x20")		1
4	SP1588273	Backrest, Matrx Elite Deep Back 16x20", cpl. w/ mounting hardware [2178+2121]		1
5	SP1588274	Backrest, Matrx Elite Deep Back 17x20", cpl. w/ mounting hardware [2178+2122]		1
6	SP1588275	Backrest, Matrx Elite Deep Back 18x20", cpl. w/ mounting hardware [2178+2123]		1

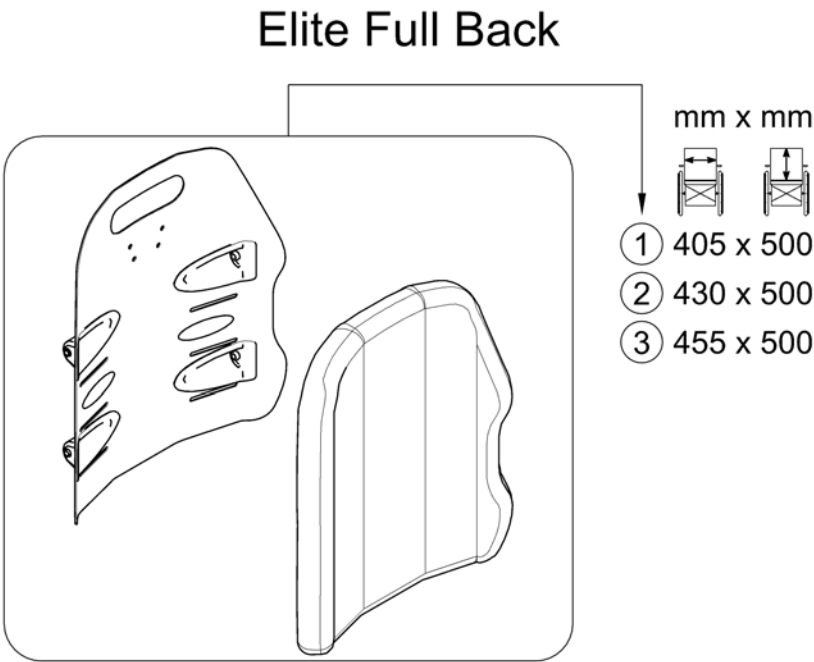

Pos.	Part number	Description	Valid from → until	Qty
1	SP1531824	Backrest, Matrx Elite (Full) Back 16x20", cpl. w/ mounting hardware [2176]		1
2	SP1531831	Backrest, Matrx Elite (Full) Back 17x20", cpl. w/ mounting hardware [2176]		1
3	SP1531832	Backrest, Matrx Elite (Full) Back 18x20", cpl. w/ mounting hardware [2176]		1

Flex3 back, tension adjustable


1602719-000

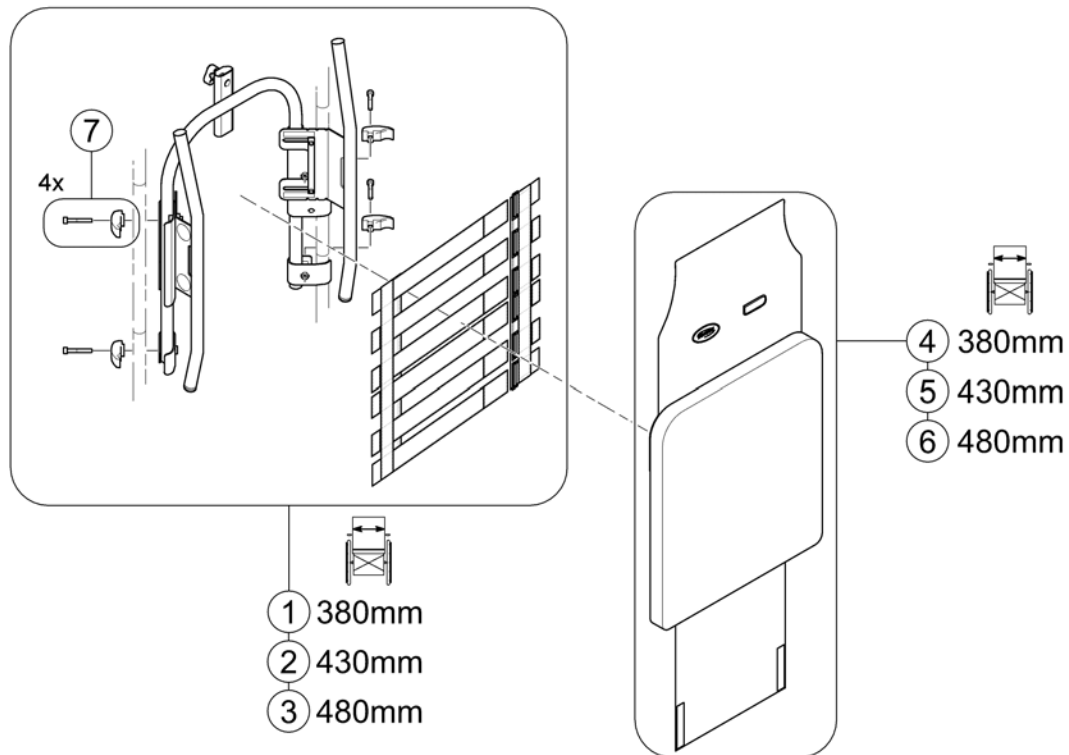

Pos.	Part number	Description	Valid from → until	Qty
1	SP1579675	No description available.		1
2	SP1579692	No description available.		1
3	SP1579693	No description available.		1
4	SP1543667	Backrest cushion Lateral 475x540mm, Dartex black (TR26)		1
5	SP1543669	Backrest cushion Lateral 525x540mm, Dartex black (TR26)		1
6	SP1543671	Backrest cushion Lateral 575x540mm, Dartex black (TR26)		1
7	SP1580557	Mounting Hardware (Kit of 4)		1

Legrest Support, Vari series 07/2005 ==>


1449734 iso

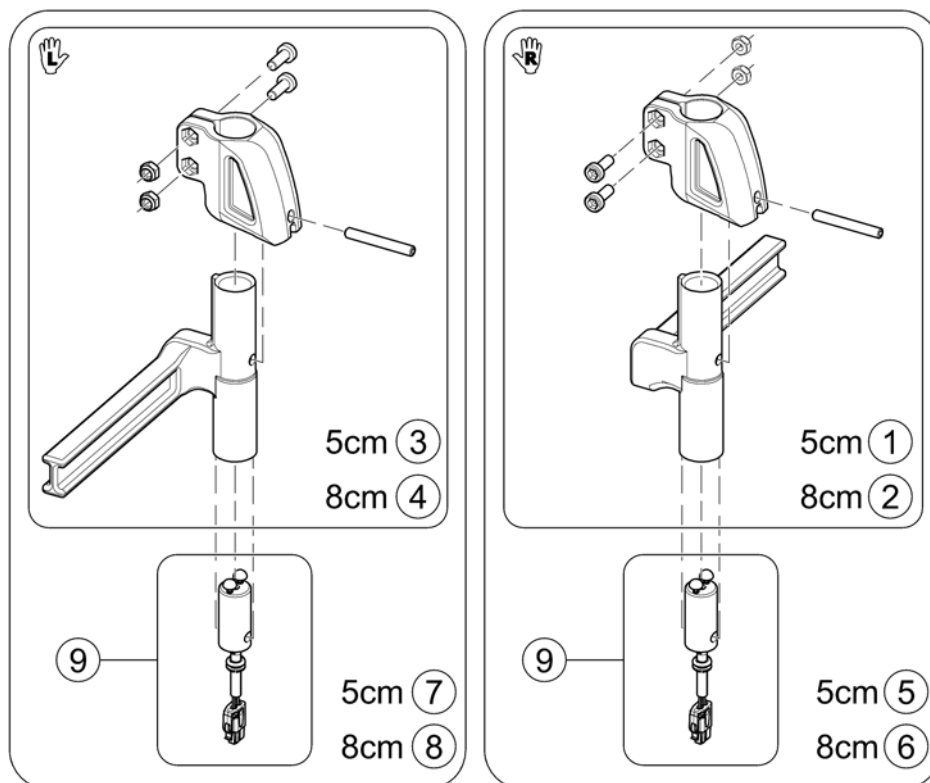


Pos.	Part number	Description	Valid from → until	Qty
1	SP1449732	Legrest Support Vari series, man. 5cm - RH		1
2	SP1449736	Footrest/Legrest receptacle, manuell, RH, 8cm		1
3	SP1449733	Legrest Support Vari series, man. 5cm - LH		1
4	SP1449737	Footrest/Legrest receptacle, manuell, LH, 8cm		1
5	SP1531071	Legrest Support Vari series, powered 5cm - RH		1
6	SP1531073	Legrest Support Vari series, powered 8cm - RH		1
7	SP1531072	Legrest Support Vari series, powered 5cm - LH		1
8	SP1531082	Legrest Support Vari series, powered 8cm - LH		1
9	SP1530795	Power Supply Cable Legrest ACS2 (1000mm)		1

Hanger Vari F

144934 / X.dwg

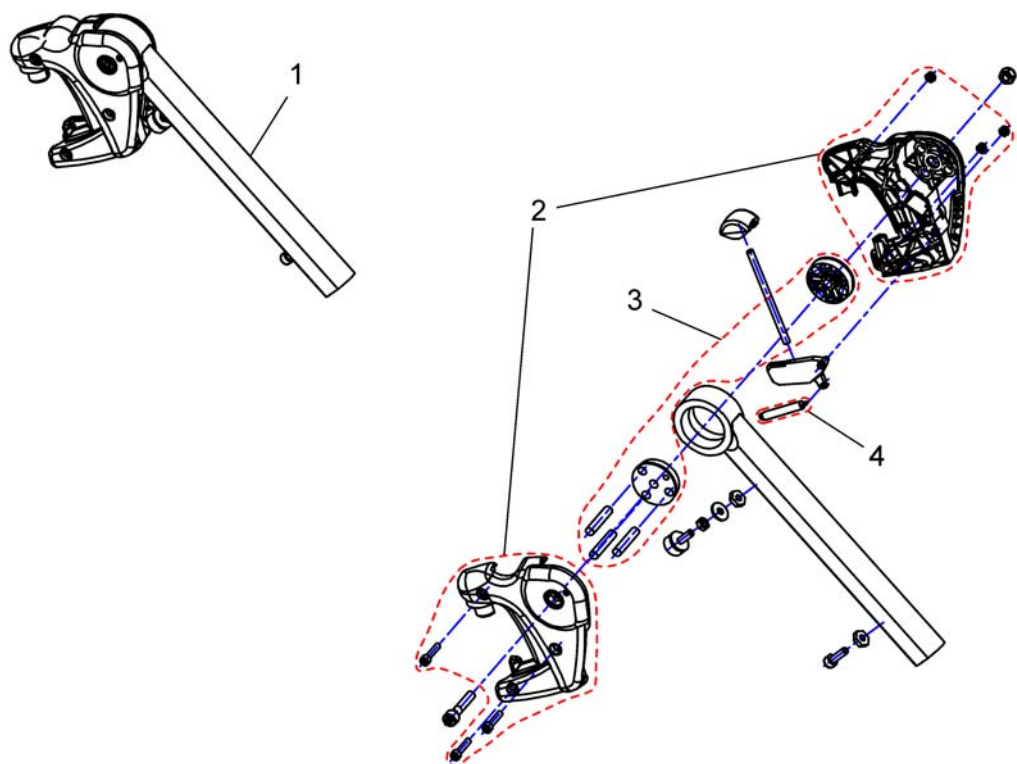


Pos.	Part number	Description	Valid from → until	Qty
1	SP1449347	Hanger Vari F LH cpl.		1
1	SP1449346	Hanger Vari F RH cpl.		1
2	SP1434955	Cover Legrest cpl.		1
3	1434956	Mechanic		1
4	SPZ173178	Spring		1

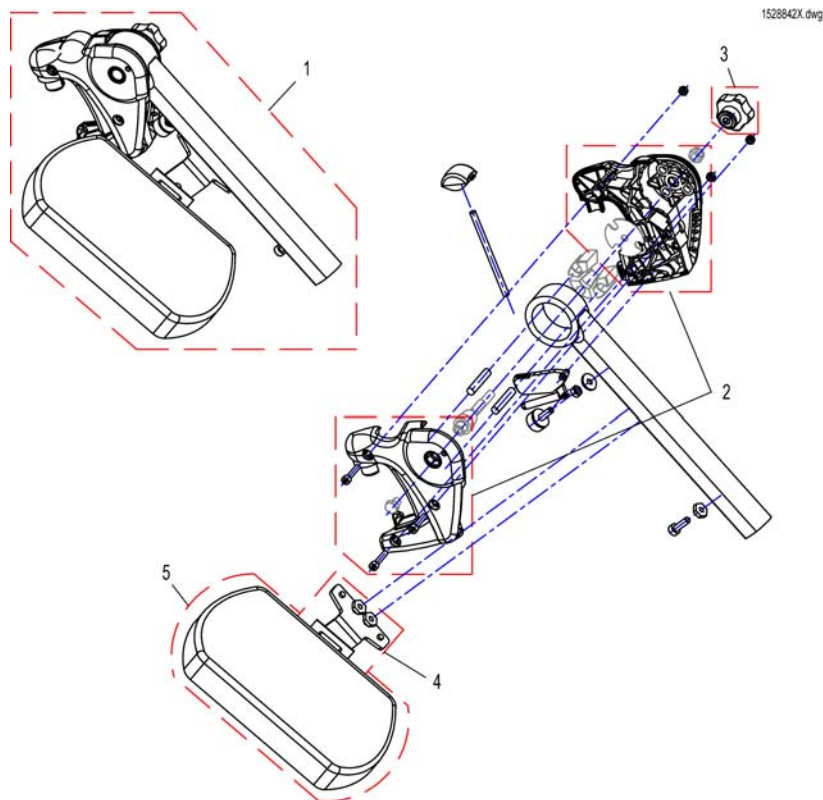Hanger Vari A (04/2009 =>)


Pos.	Part number	Description	Valid from → until	Qty
-	SP1527820	Rework-Kit Vari A Legrest Version 1 => Version 2 <i>Vari A Version 1 => Version 2 Upgrade Kit</i>		1
1	SP1527498	Hanger Vari A LH, cpl.		1
1	SP1527492	Hanger Vari A RH, cpl.		1
2	SP1434955	Cover Legrest cpl.		1
3	L34011	Handknob M8 Tristarfemale		1
4	SP1450447	Calf Pad Holder Left Turnable, cpl.		1
4	SP1450446	Calf Pad Holder Right Turnable, cpl.		1
5	6301333	Legrest calf pad		1

Hangers ADM LH/RH manual

1449409X.dwg

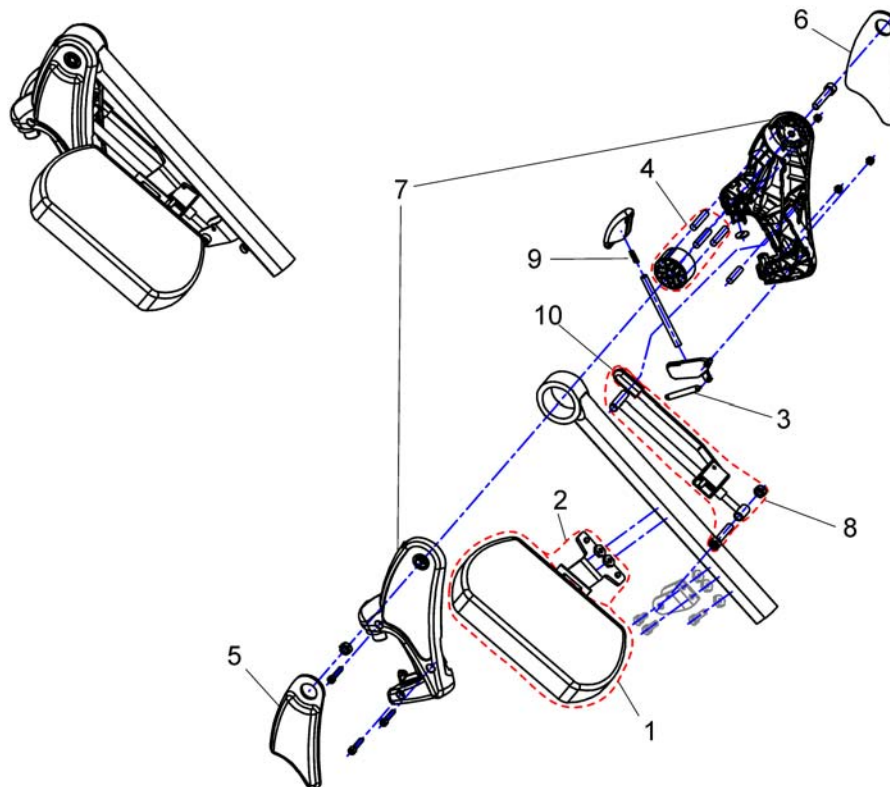


Pos.	Part number	Description	Valid from → until	Qty
1	6301333	Legrest calf pad		1
2	SP1450447	Calf Pad Holder Left Turnable, cpl.		1
2	SP1450446	Calf Pad Holder Right Turnable, cpl.		1
3	SPZ173178	Spring		1
4	SPZ174134	Floating Cylinder With Pin		1
5	SP1450410	Knee Upholstery LH		1
6	SP1450448	Knee Upholstery RH		1
7	SP1449293	Ad-Cover cpl.		1
8	SPZ174130	ADM Adjustment mechanism cpl.		1
9	SPN12196	Spring		1
10	SP1006490	Cap for tripping lever		1
11	SP1449410	Hanger ADM RH cpl.		1
11	SP1449409	Hanger ADM LH cpl.		1

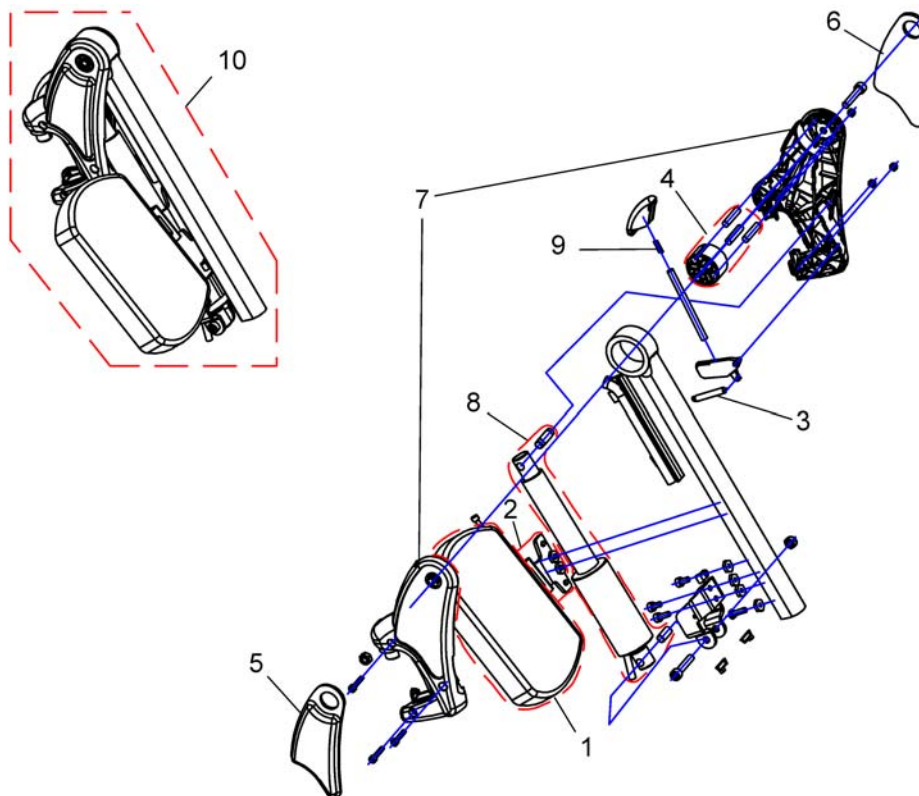Hanger Vari ADE/ADP electric (07/2005 =>)

1449407X.dwg

Pos.	Part number	Description	Valid from → until	Qty
1	6301333	Legrest calf pad		1
2	SP1450447	Calf Pad Holder Left Turnable, cpl.		1
2	SP1450446	Calf Pad Holder Right Turnable, cpl.		1
3	SPZ173178	Spring		1
4	SPZ174134	Floating Cylinder With Pin		1
5	SP1450448	Knee Upholstery RH		1
6	SP1450410	Knee Upholstery LH		1
7	SP1449293	Ad-Cover cpl.		1
8	SPZ151006	Actuator with circuit board, preassembled		1
9	SPN12196	Spring		1
10	SP1449408	Hanger ADP RH cpl.		1
10	SP1449407	Hanger ADP LH cpl.		1

Calfpad Holder (07/2005 =>)

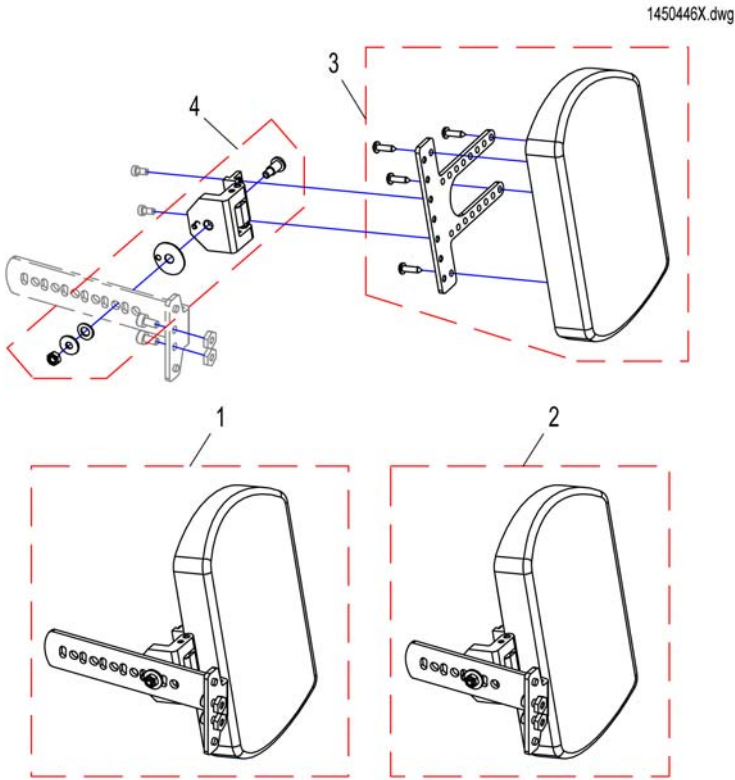


Pos.	Part number	Description	Valid from → until	Qty
1	SP1450447	Calf Pad Holder Left Turnable, cpl.		1
1	SP1450446	Calf Pad Holder Right Turnable, cpl.		1
2	SP1487409	Calf Plate Holder for Recaroseat RH		1
2	SP1487410	Calf Plate Holder for Recaroseat LH		1
3	SP1449021	Assembly Kit, Calf Plate Holder		1
4	SP1450226	CALF PLATE HINGE, RH		1
4	SP1450225	CALF PLATE HINGE, LH		1

Central legrests - Std-/ Easy Adapt Seat

Pos.	Part number	Description	Valid from → until	Qty
-		Support central legrests - Std-/ Easy Adapt Seat		1
-		Central Legrest man.		1
-		Centr. Legrest pow. LNX		1

Support central legrests - Std-/ Easy Adapt Seat


1541397X.iso

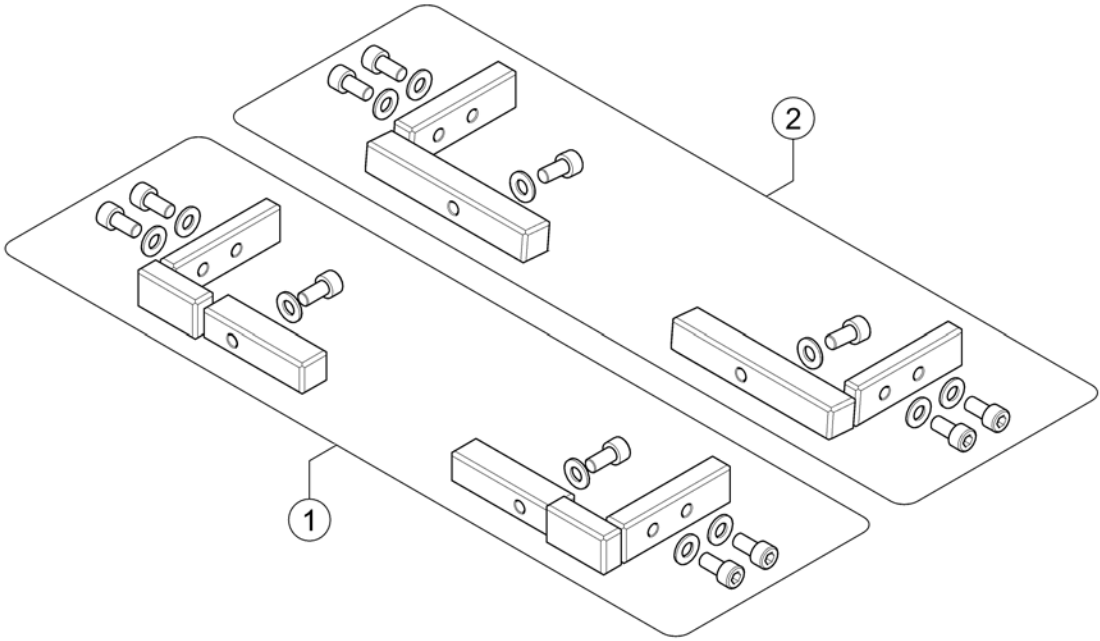


Pos.	Part number	Description	Valid from → until	Qty
1	SP1541397	Support center mounted legrest 430-530 mm		1
1	SP1541398	Support center mounted legrest 380-410 mm		1

Central Legrest man.

1583896 Rev

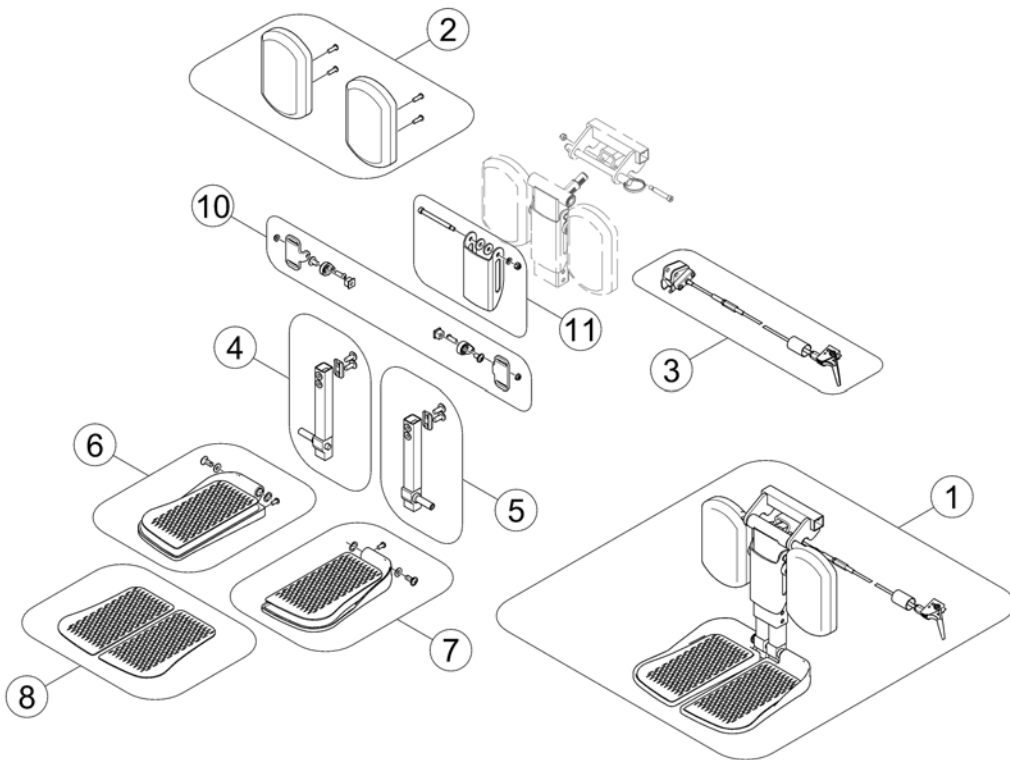

Pos.	Part number	Description	Valid from → until	Qty
1	SP1537926	Central Legrest manual, cpl.		1
2	SP1163373	Calfplates (Pair)		1
3	SP1121079	Bowden cable central legrest, manual		1
4	SP1161986	Kit, extension tube, cpl. RH		1
5	SP1161987	Kit, extension tube, cpl. LH		1
6	SP1161990	Footplate, cpl. RH		1
7	SP1161991	Footplate, cpl. LH		1
8	SP1163375	Footrest pad (Pair)		1
10	SP1165505	Kit, Adjustment Hardware Calf Pads		1
11	SP1165506	Kit, Calf Pad mounting bracket w/ hardware		1

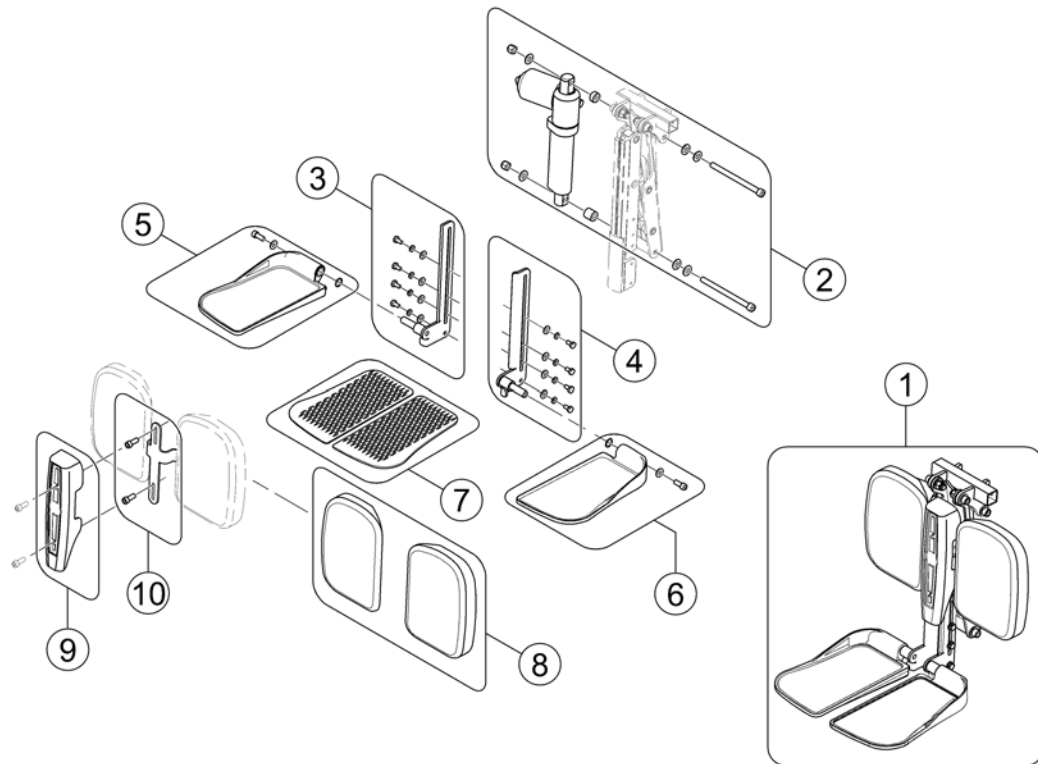


Pos.	Part number	Description	Valid from → until	Qty
-	SP1541677	Adaptercable 300mm <i>For chairs with lifter</i>		1
-	SP1543012	Adaptercable 1200mm <i>For chairs without lifter</i>		1
1	SP1537489	Centre Mounted Legrest LNX (Standard / Easy Adapt) complete		1
2	SP1543031	Actuator, cpl. with mounting hardware		1
3	SP1543035	Footplate mount, RH		1
4	SP1543034	Footplate mount, LH		1
5	SP1543037	Footplate, RH		1
6	SP1543036	Footplate, LH		1
7	SP1163375	Footrest pad (Pair)		1
8	SP1543033	Calfpads (Pair)		1
9	SP1543032	Cover		1
10	SP1541397	Support center mounted legrest 430-530 mm		1
11	SP1541398	Support center mounted legrest 380-410 mm		1
12	SP1579985	Calf Plate Bracket		1

Central legrests - Modulite Seat

Pos.	Part number	Description	Valid from → until	Qty
-		Support Centr. Legrest man. - Modulite Seat		1
-		Support Centr. Legrest LNX pow. - Modulite Seat		1
-		Central Legrest man.		1
-		Central Legrest pow. LNX		1

Support Centr. Legrest man. - Modulite Seat


1563891 Rev

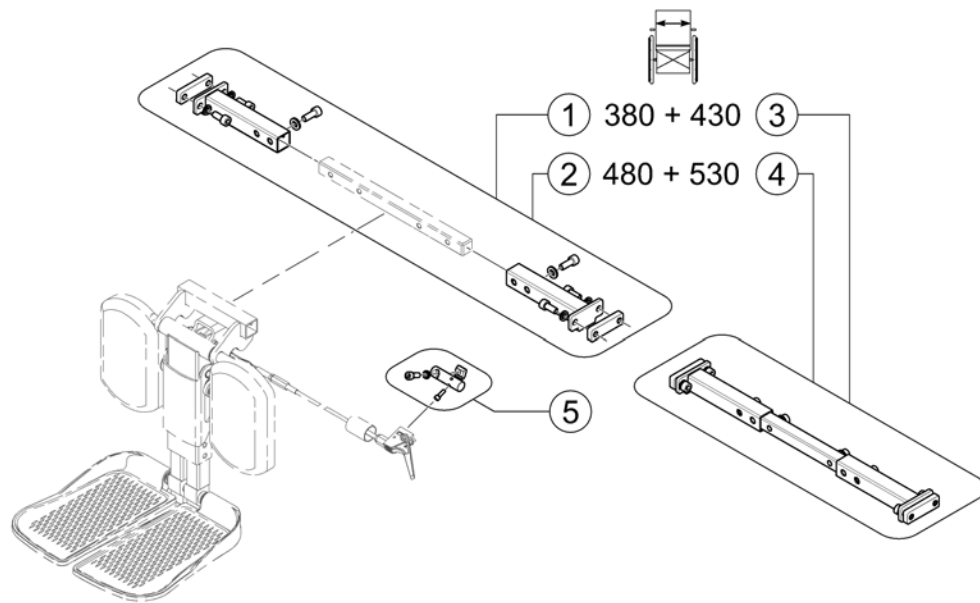


Pos.	Part number	Description	Valid from → until	Qty
1	SP1579989	Support, Centr. Legrest man. - SW 380+430mm		1
2	SP1579990	Support, Centr. Legrest man. LNX - SW 480+530mm		1
3	SP1579991	Support, Centr. Legrest man. LNX - SW 380+430mm, cpl.		1
4	SP1579992	Support, Centr. Legrest man. LNX - SW 480+530mm, cpl.		1
5	SP1579988	Support, Lever, cpl.		1

Support Centr. Legrest LNX pow. - Modulite Seat

1563893.kso

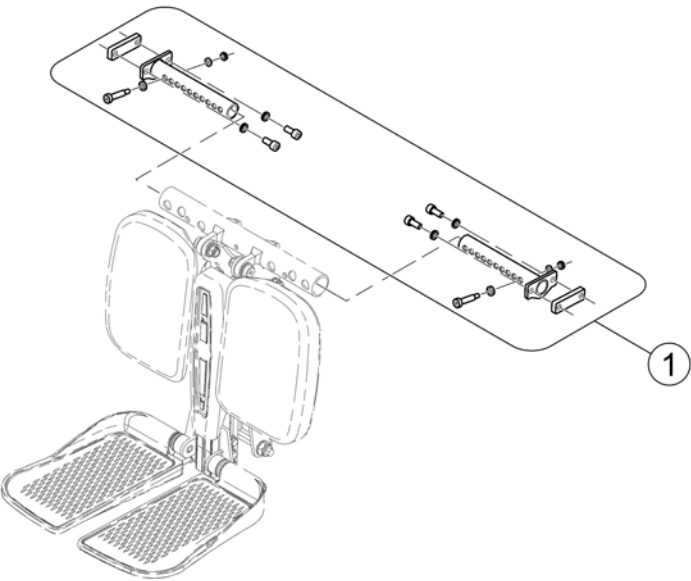


Pos.	Part number	Description	Valid from → until	Qty
1	SP1575144	Support, Centr. Legrest el. LNX - Modulite		1

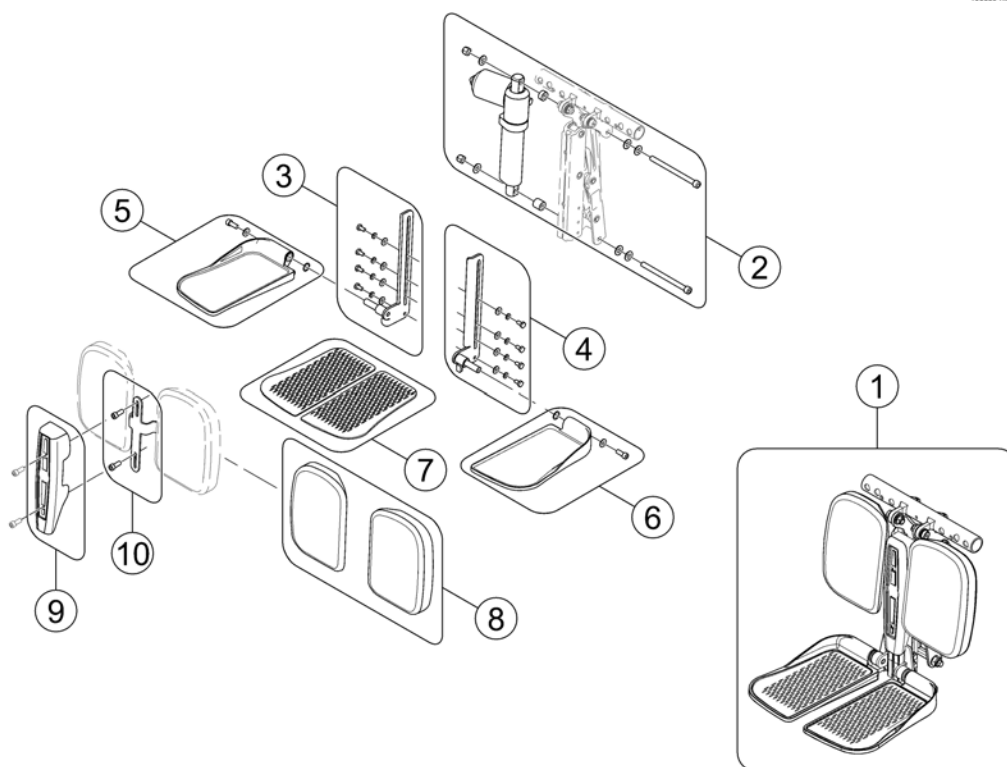

Pos.	Part number	Description	Valid from → until	Qty
-	SP1541677	Adaptercable 300mm <i>For chairs with lifter</i>		1
-	SP1543012	Adaptercable 1200mm <i>For chairs without lifter</i>		1
-	SP1571959	Central Legrest pow. LNX for Modulite cpl. (incl. Support)		1
1	SP1571882	Central Legrest pow. LNX		1
2	SP1543031	Actuator, cpl. with mounting hardware		1
3	SP1543035	Footplate mount, RH		1
4	SP1543034	Footplate mount, LH		1
5	SP1543037	Footplate, RH		1
6	SP1543036	Footplate, LH		1
7	SP1163375	Footrest pad (Pair)		1
8	SP1543033	Calfpads (Pair)		1
9	SP1543032	Cover		1
10	SP1579985	Calf Plate Bracket		1


Footrests Vari series

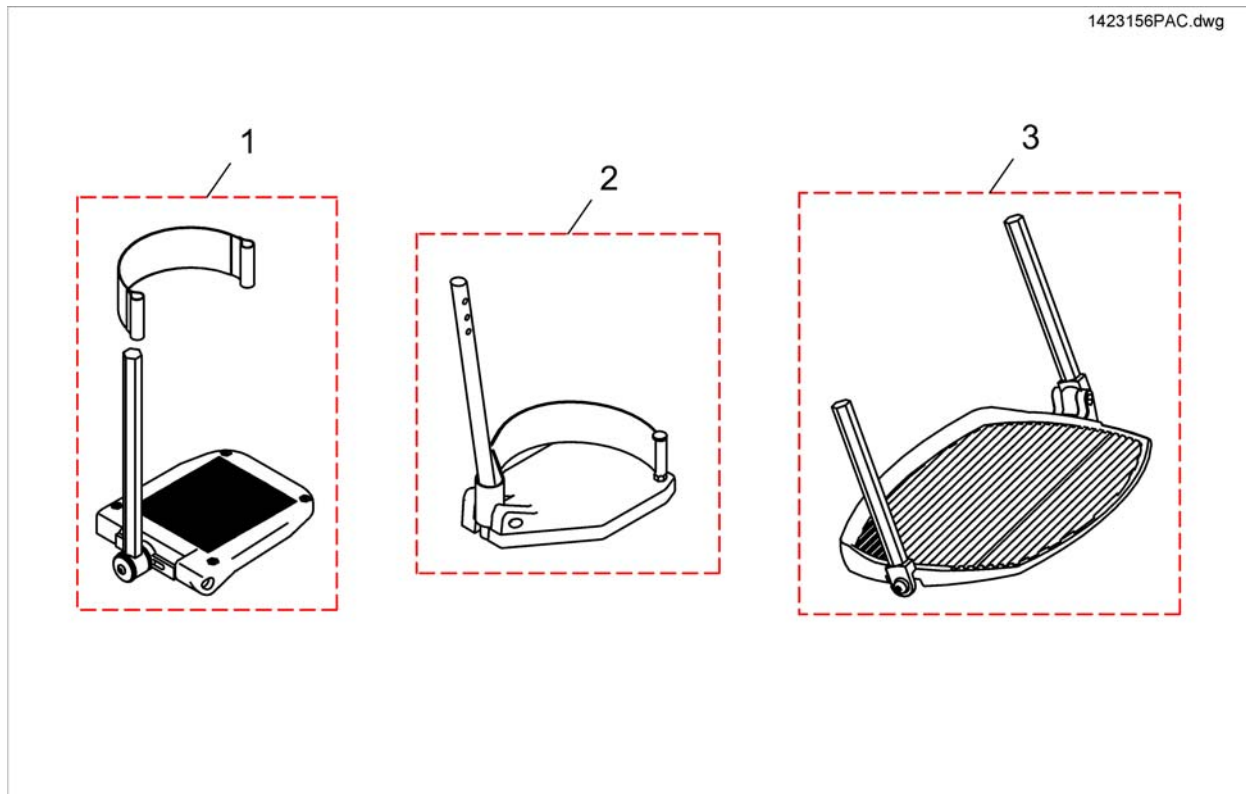

Pos.	Part number	Description	Valid from → until	Qty
1	SP1419344	Footplate LH Anlge And Depth Adjustable, With Support Tube cpl.		1
1	SP1419345	Footplate RH Anlge And Depth Adjustable, With Support Tube cpl.		1
2	SP1421662	Footplate RH fix, with Support Tube cpl.		1
2	SP1421663	Footplate LH fix, with Support Tube cpl.		1
3	SP1423156	Footboard, pre-assembled		1


Invacare® FDX | Footplates / Support tubes
Footrests Vari, Flip-Up (01.08.10 =>)

1529645X.jpg

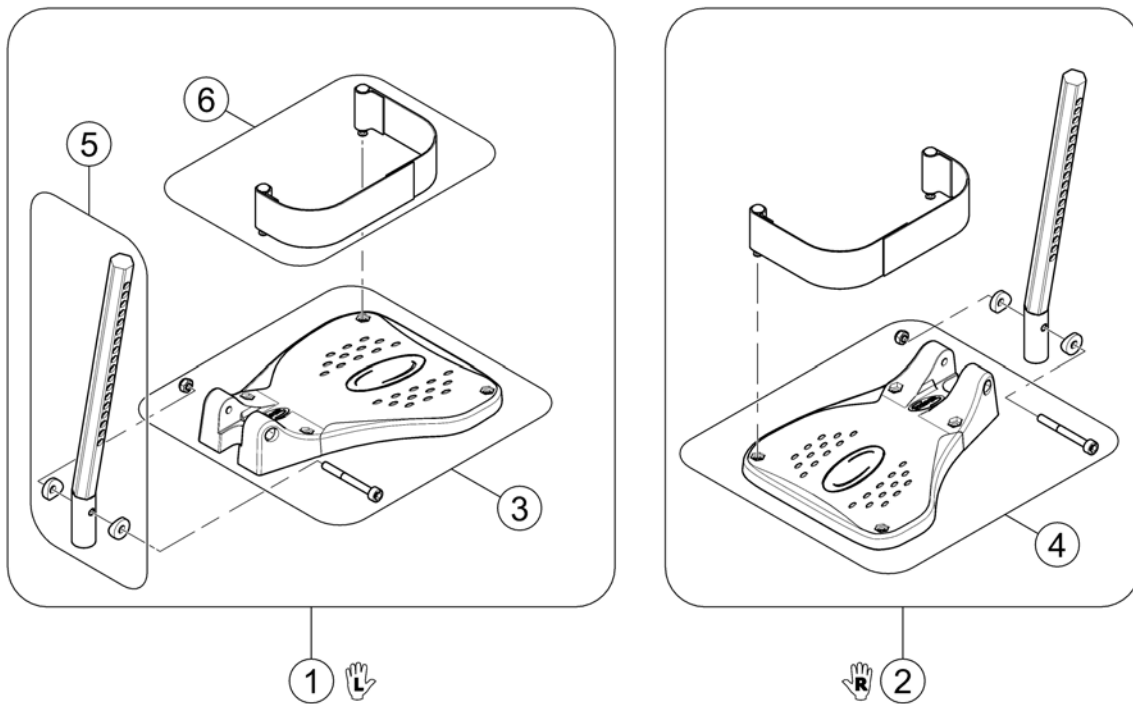

Pos.	Part number	Description	Valid from → until	Qty
1	SP1541575	Footplate flip up, LH, cpl.		1
2	SP1541574	Footplate flip up, RH, cpl.		1
3	SP1529645	FOOTPLATE LH		1
4	SP1529644	FOOTPLATE RH		1
5	SP1543045	Plug-in tube LH/RH		1
6	SP1442467	Heel Strap cpl.		1


Footrests Vari, Flip-Up & angle adjustable (01.08.10 =>)

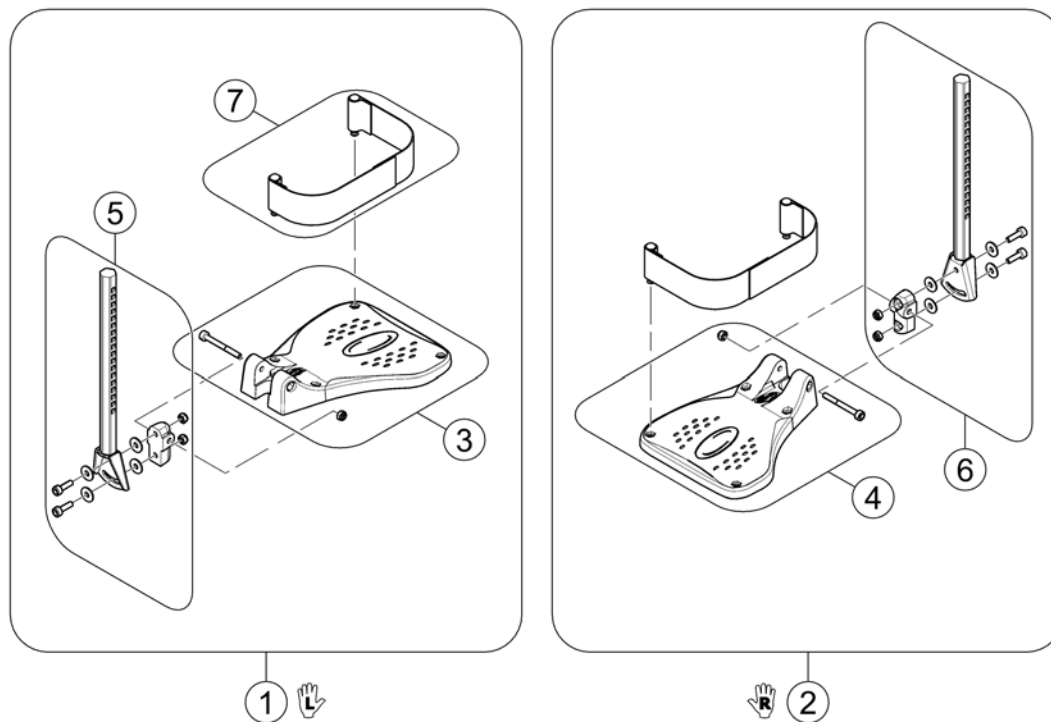

Pos.	Part number	Description	Valid from → until	Qty
1	SP1541580	Footplate flip up, angle adjustable, LH, cpl.		1
2	SP1541579	Footplate flip up, angle adjustable, RH, cpl.		1
3	SP1529645	FOOTPLATE LH		1
4	SP1529644	FOOTPLATE RH		1
5	SP1423204	FOOT PLATE TUBE, ANGLE ADJUSTMENT LH CPL.		1
6	SP1423203	FOOT PLATE TUBE, ANGLE ADJUSTMENT RH CPL.		1
7	SP1442467	Heel Strap cpl.		1

Armrests


SPCFDX20 160

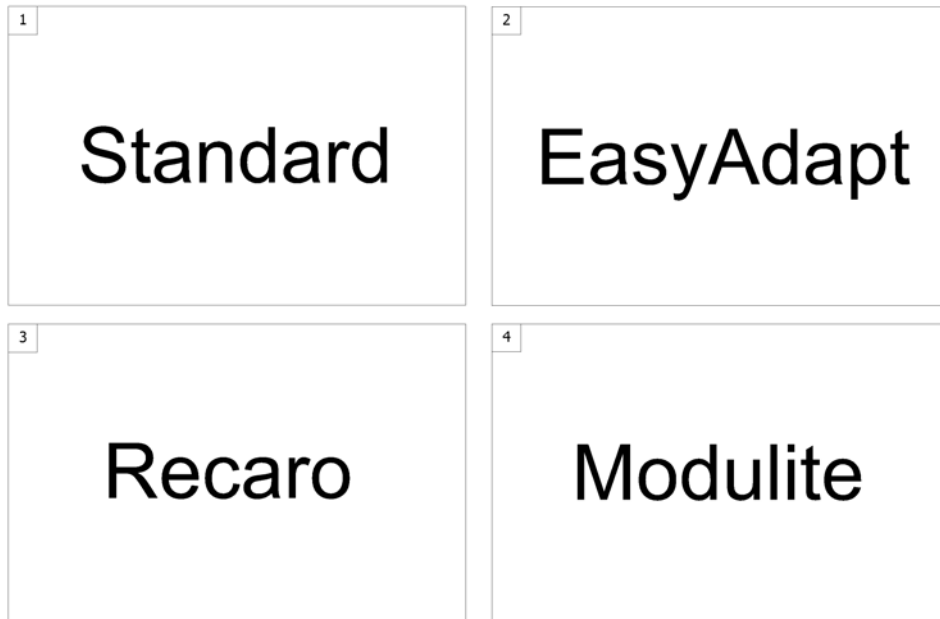

Pos.	Part number	Description	Valid from → until	Qty
1	-	Armrest for Standard Seat		1
2	-	Armrests for EasyAdapt Seat		1
3	-	Armrest for Recaro Seat		1
4	-	Armrests for Modulite Seat		1

Armrest for Standard Seat


1573232.iss

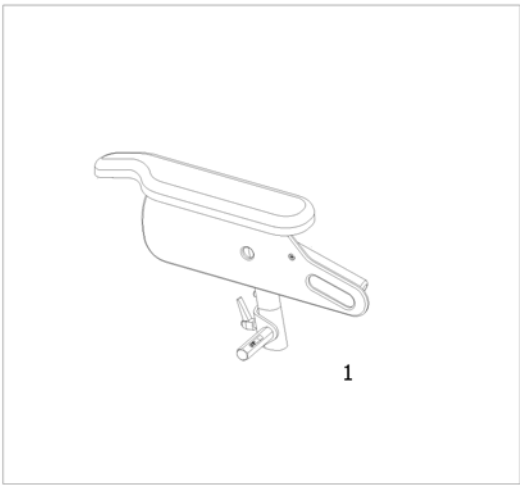

Pos.	Part number	Description	Valid from → until	Qty
1	-	Fixed Armrest, removable [0635/0636]		1

Fixed Armrest, removable


1421692X.dwg

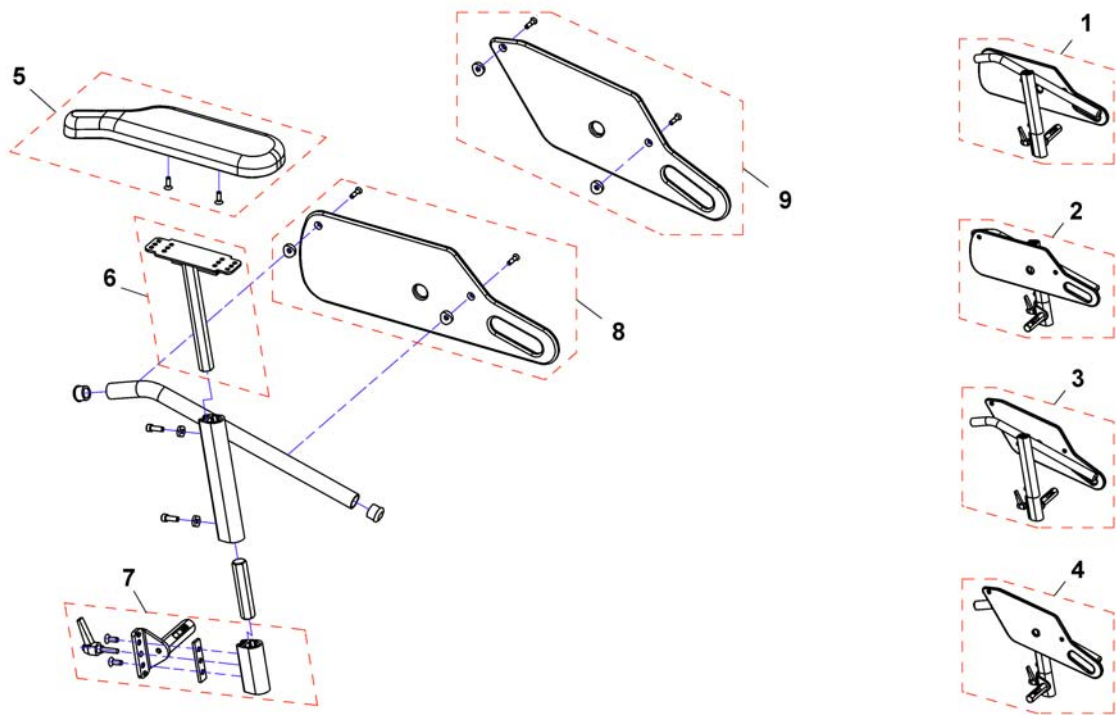

Pos.	Part number	Description	Valid from → until	Qty
1	SP1421692	Armrest, height max. 340mm, RH, Seatdepth 460mm & 510mm		1
1	SP1421693	Armrest, height max. 380mm, RH, Seatdepth 460mm & 510mm		1
2	SP1421694	Armrest, height max. 340mm, LH, Seatdepth 460mm & 510mm		1
2	SP1421695	Armrest, height max. 380mm, LH, Seatdepth 460mm & 510mm		1
3	SP1436533	Armrest, height max. 340mm, RH, Seatdepth 410mm		1
3	SP1436535	Armrest, height max. 380mm, RH, Seatdepth 410mm		1
4	SP1436534	Armrest, height max. 340mm, LH, Seatdepth 410mm		1
4	SP1436536	Armrest, height max. 380mm, LH, Seatdepth 410mm		1
5	SP1449043	Ampad, Standard (325mm), RH		1
5	SP1443004	Ampad, Standard (325mm), LH		1
5	SP1528845	Ampad, Extended (375mm), RH		1
5	SP1528846	Ampad, Extended (375mm), LH		1
6	SP1510856	Armrest holder RH <i>For chairs bevor 01/01/09 also order a new armpad.</i>		1
6	SP1510857	Armrest holder LH <i>For chairs bevor 01/01/09 also order a new armpad.</i>		1
7	SP1423223	Side part receptacle RH, cpl.		1
7	SP1423225	Side part receptacle LH, cpl.		1
8	SP1420471	Skirtguard, Seatdepth 460 & 510mm		1
9	SP1428078	Skirtguard, seat depth 410mm		1

Armrests for EasyAdapt Seat


1573233 iso

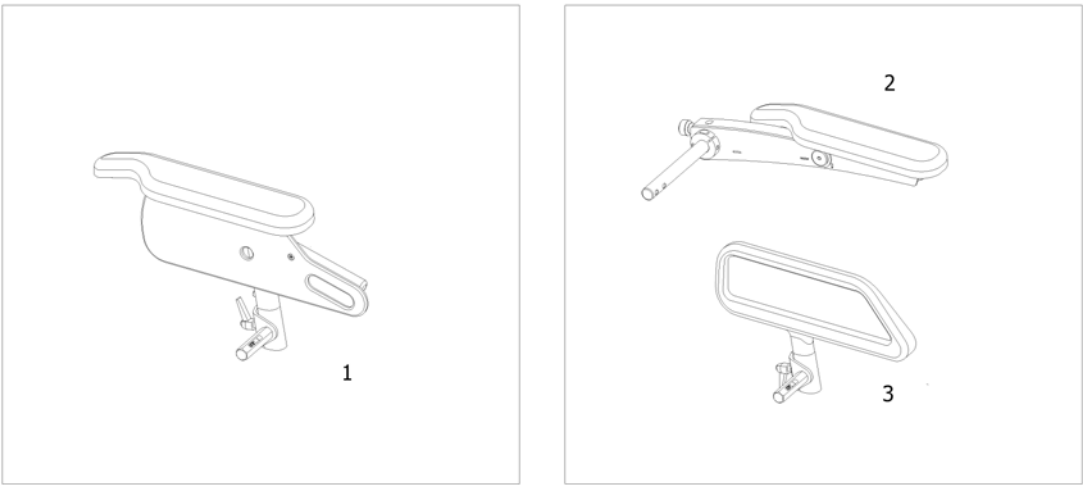

Pos.	Part number	Description	Valid from → until	Qty
1	-	Fixed Armrest, removable [0635/0636]		1
2	-	Following armrests [0620]		1
3	-	Skirtguard for following armrest [0623]		1

Skirtguard for following armrest


1425860X.dwg

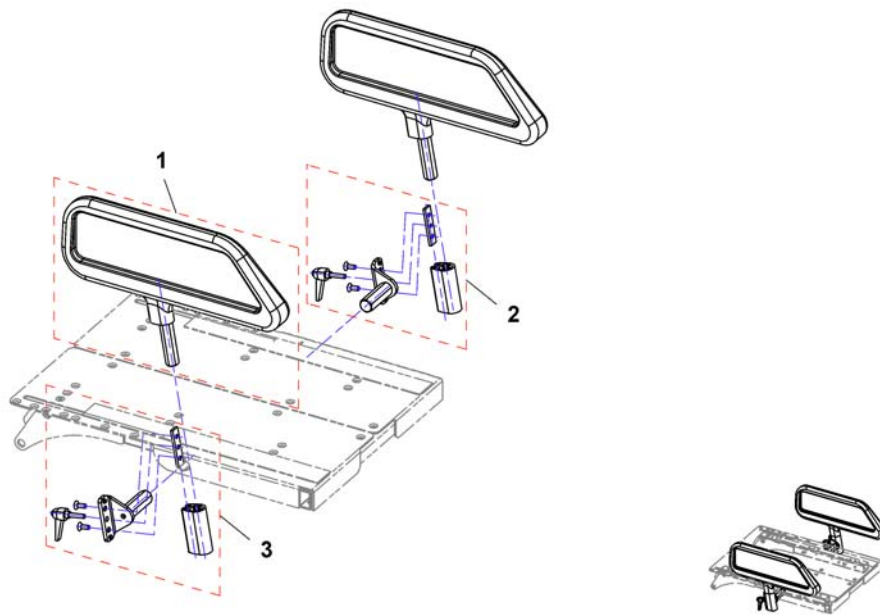

Pos.	Part number	Description	Valid from → until	Qty
1	SP1426303	Skirtguard		1
2	SP1442549	Bracket for Armrest / Skirtguard LH		1
3	SP1442550	Bracket for Armrest / Skirtguard RH		1

Armrest for Recaro Seat


1573232.iss

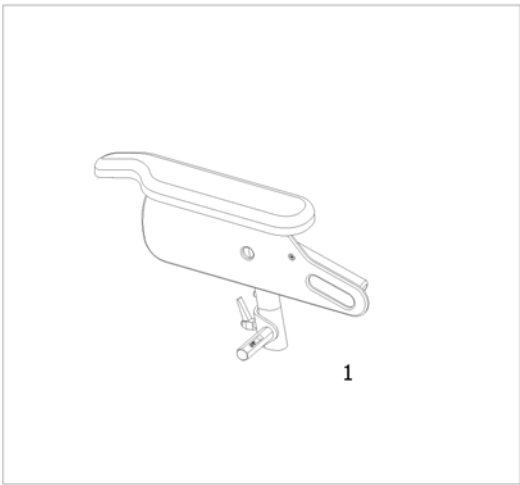

Pos.	Part number	Description	Valid from → until	Qty
1	-	Fixed Armrest, removable [0636]		1

Armrests for Modulite Seat


1571738.iso

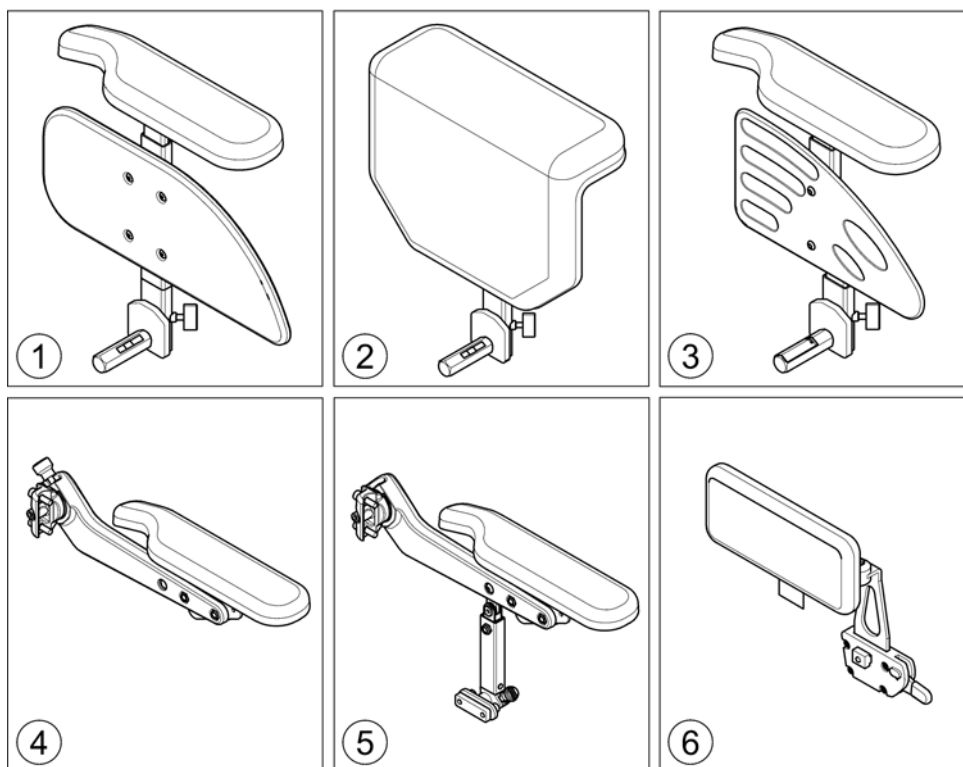

Pos.	Part number	Description	Valid from → until	Qty
1	-	T-Armrest [2191+2195/2196/2197]		1
2	-	Padded Armrest [2198]		1
3	-	Inrigged Armrest [2183+2195/2196/2197/2199]		0
4	-	Flip-Up Armrest [2193+2195/2196/2197/2199]		0
5	-	Following Armrest [2192/2194+2195/2196/2197/2199]		0
6	-	Hip Support [2184/2185/2186/2187/2188/2189]		0

T-Armrest


1572570 iso

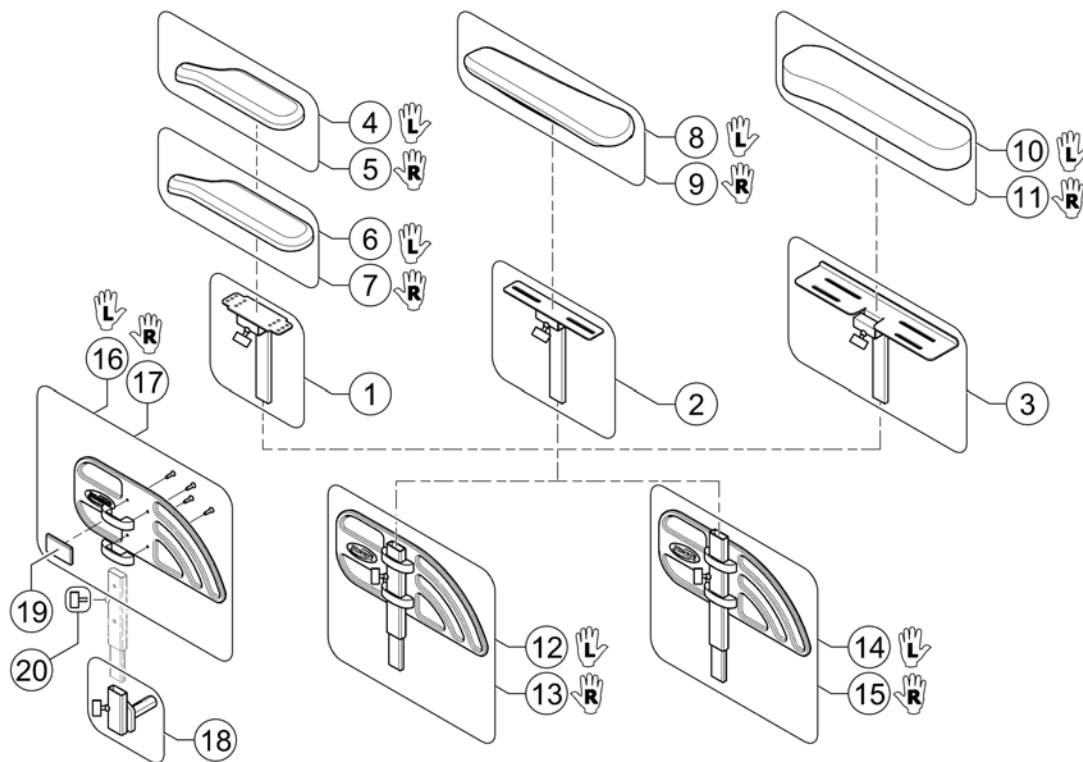

Pos.	Part number	Description	Valid from → until	Qty
1	SP1567750	Armrest Bracket for Standard & Extended Armpad		1
2	SP1570319	Armrest Bracket for Ergonomic Armpad		1
3	SP1585088	Armrest Bracket for Soft Armpad		1
4	SP1443004	Armpad, Standard (325mm), LH		1
5	SP1449043	Armpad, Standard (325mm), RH		1
6	SP1528846	Armpad, Extended (375mm), LH		1
7	SP1528845	Armpad, Extended (375mm), RH		1
8	SP1571589	Armpad, Ergonomic, LH		1
9	SP1571588	Armpad, Ergonomic, RH		1
10	SP1585068	Armpad, Soft, LH		1
11	SP1585069	Armpad, Soft, RH		1
12	SP1588374	Siderest LH, cpl. w/o brackets		1
13	SP1588375	Siderest RH, cpl. w/o brackets		1
14	-	Not available so far.		0
15	-	Not available so far.		0
16	SP1572940	Skirt Guard LH		1
17	SP1572948	Skirt Guard RH		1
18	SP1567749	Siderest Bracket - LH/RH		1
19	SPP31298	Reflector, yellow		1
20	SPL10024	Wing Screw M6x16		1

Padded Armrest


1573214/00

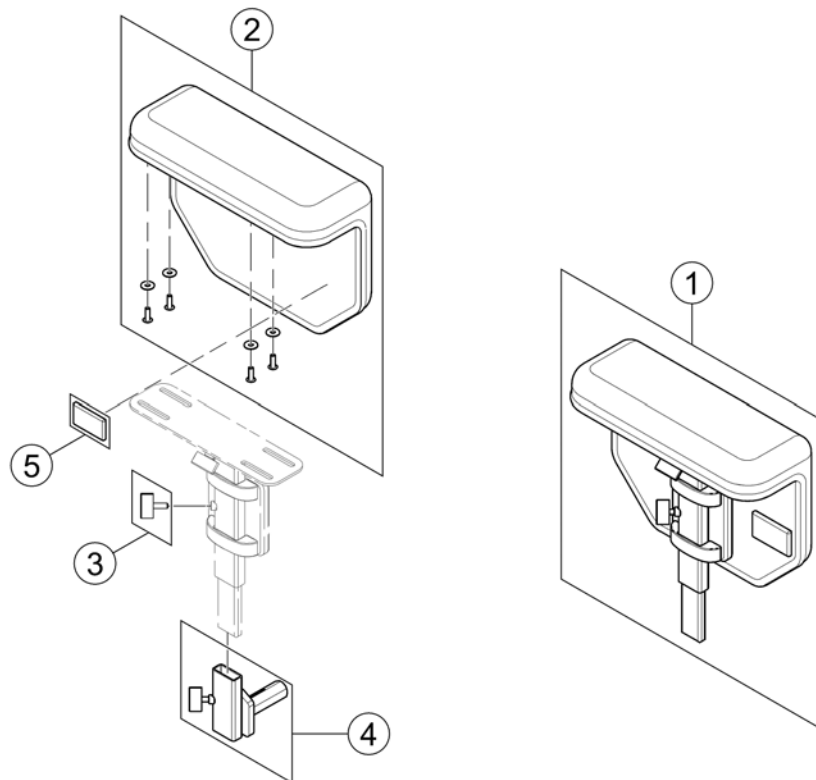

Pos.	Part number	Description	Valid from → until	Qty
1	SP1571987	Padded Armrest RH/LH, cpl.		1
2	SP1571962	Armpad, padded - RH/LH with integrated cloth guard		1
3	SPL10024	Wing Screw M6x16		1
4	SP1567749	Siderest Bracket - LH/RH		1
5	SPP31298	Reflector, yellow		1


Inrigged Armrest

1585180 iso

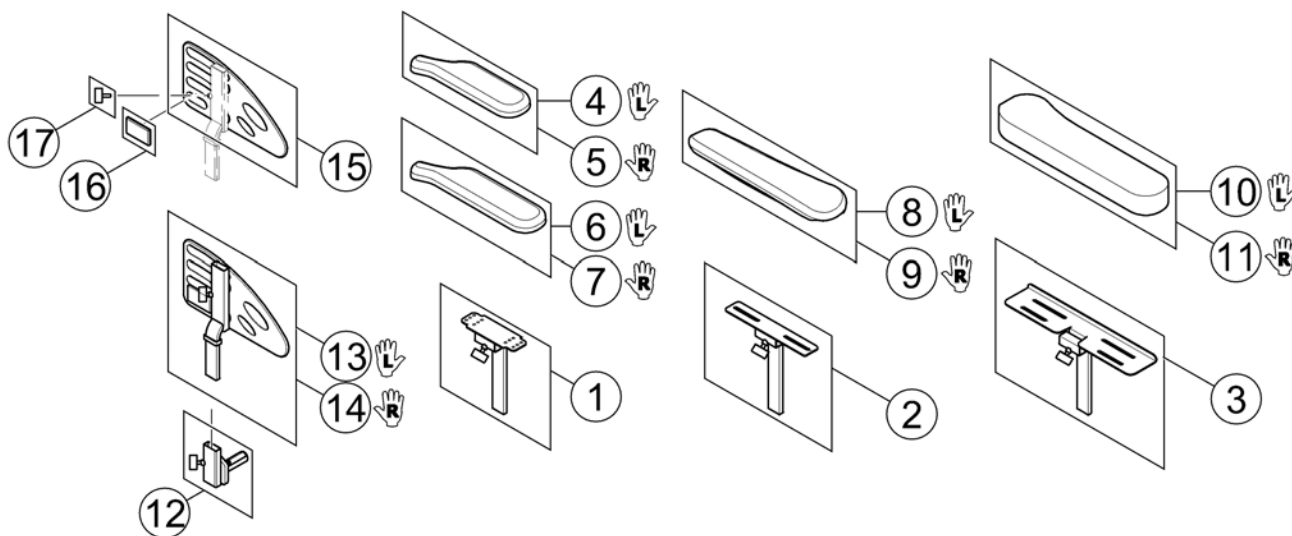


Pos.	Part number	Description	Valid from → until	Qty
1	SP1567750	Armrest Bracket for Standard & Extended Armpad		1
2	SP1570319	Armrest Bracket for Ergonomic Armpad		1
3	SP1585088	Armrest Bracket for Soft Armpad		1
4	SP1443004	Armpad, Standard (325mm), LH		1
5	SP1449043	Armpad, Standard (325mm), RH		1
6	SP1528846	Armpad, Extended (375mm), LH		1
7	SP1528845	Armpad, Extended (375mm), RH		1
8	SP1571589	Armpad, Ergonomic, LH		1
9	SP1571588	Armpad, Ergonomic, RH		1
10	SP1585068	Armpad, Soft, LH		1
11	SP1585069	Armpad, Soft, RH		1
12	SP1567749	Siderest Bracket - LH/RH		1
13	SP1585037	Siderest inrigged LH		1
14	SP1585038	Siderest inrigged RH		1
15	SP1578119	Skirt Guard LH/RH		1
16	SPP31298	Reflector, yellow		1
17	SPL10024	Wing Screw M6x16		1

Flip-Up Armrest

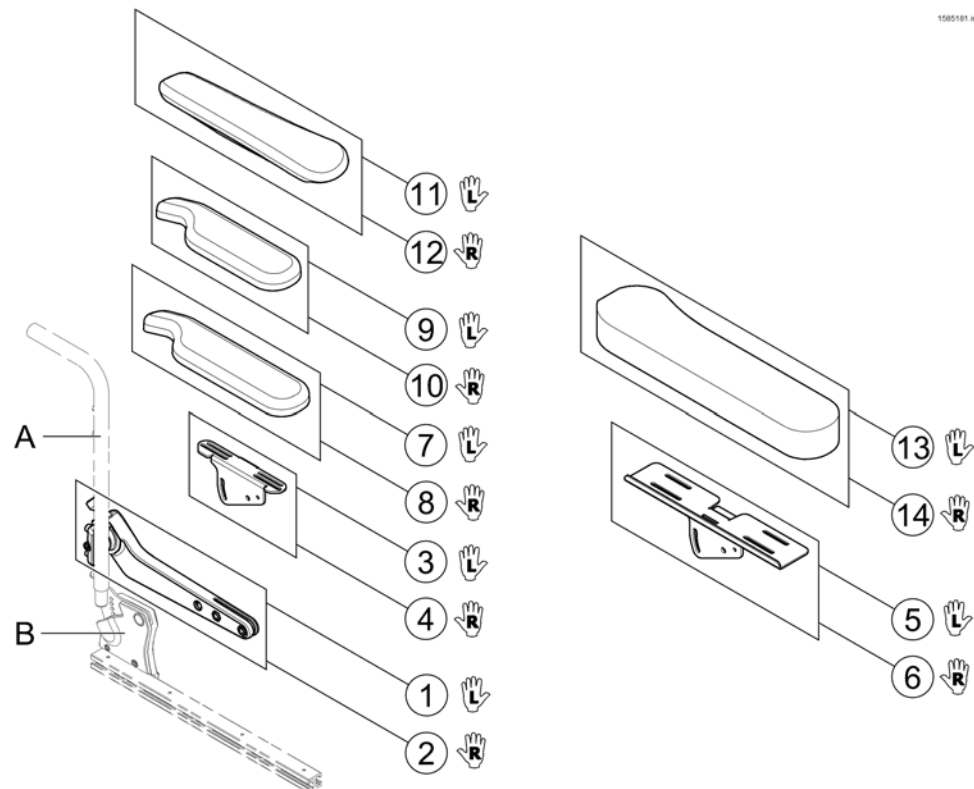

Pos.	Part number	Description	Valid from → until	Qty
-	-	Retrofitting the armrests on chairs built before 15-03-27 also requires new Pos. A & B and Sling Back with respective cut-outs! !!!		0
1	SP1582668	Flip-Up Armrest, LH	2015-01-19 →	1
2	SP1582644	Flip-Up Armrest, RH	2015-01-19 →	1
3	SP1585412	Ampad Support (Standard, Extended, Ergonomic), LH		1
4	SP1585413	Ampad Support (Standard, Extended, Ergonomic), RH		1
5	SP1585414	Ampad Support (Soft), LH		1
6	SP1585415	Ampad Support (Soft), RH		1
7	SP1528846	Ampad, Extended (375mm), LH		1
8	SP1528845	Ampad, Extended (375mm), RH		1
9	SP1443004	Ampad, Standard (325mm), LH		1
10	SP1449043	Ampad, Standard (325mm), RH		1
11	SP1571589	Ampad, Ergonomic, LH		1
12	SP1571588	Ampad, Ergonomic, RH		1
13	SP1585068	Ampad, Soft, LH		1
14	SP1585069	Ampad, Soft, RH		1


Following Armrest

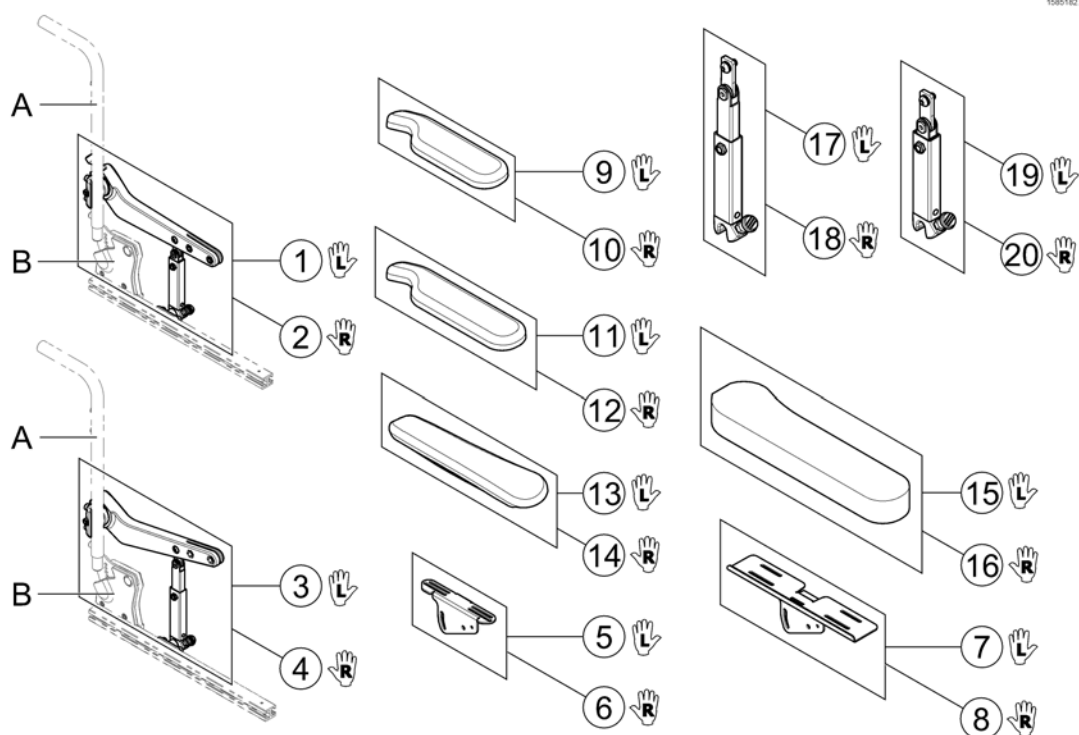


Pos.	Part number	Description	Valid from → until	Qty
-	-	Retrofitting the armrests on chairs built before 15-03-27 also requires new Pos. A & B and Sling Back with respective cut-outs! !!!		0
1	SP1585263	Armrest LH cpl. - Short Support	2015-01-19 →	1
2	SP1585264	Armrest RH cpl. - Short Support	2015-01-19 →	1
3	SP1585267	Armrest LH cpl. - Long Support	2015-01-19 →	1
4	SP1585268	Armrest RH cpl. - Long Support	2015-01-19 →	1
5	SP1585412	Ampad Support (Standard, Extended, Ergonomic), LH		1
6	SP1585413	Ampad Support (Standard, Extended, Ergonomic), RH		1
7	SP1585414	Ampad Support (Soft), LH		1
8	SP1585415	Ampad Support (Soft), RH		1
9	SP1443004	Ampad, Standard (325mm), LH		1
10	SP1449043	Ampad, Standard (325mm), RH		1
11	SP1528846	Ampad, Extended (375mm), LH		1
12	SP1528845	Ampad, Extended (375mm), RH		1
13	SP1571589	Ampad, Ergonomic, LH		1
14	SP1571588	Ampad, Ergonomic, RH		1
15	SP1585068	Ampad, Soft, LH		1
16	SP1585069	Ampad, Soft, RH		1
17	SP1583156	Armrest Support, long, LH		1
18	SP1583155	Armrest Support, long, RH		1
19	SP1583154	Armrest Support, short, LH		1
20	SP1582639	Armrest Support, short, RH		1

Hip Support

1585183 (6)

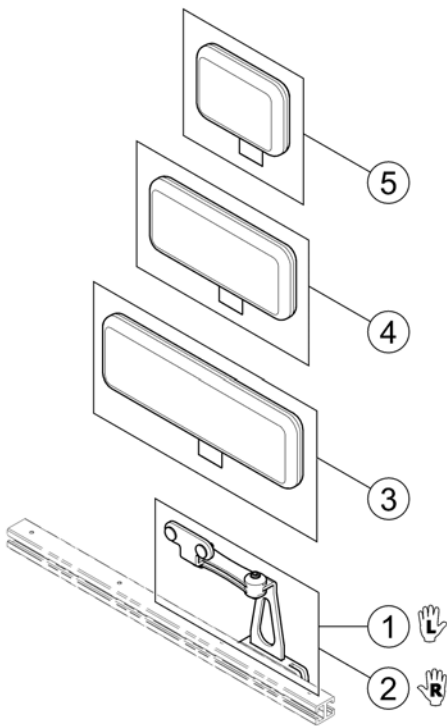


Pos.	Part number	Description	Valid from → until	Qty
1	SP1584411	Hip Support, Quick Release bracket, LH		1
2	SP1584410	Hip Support, Quick Release bracket, RH		1
3	SP1584414	Hip Support Pad, large (330x100 mm)		1
4	SP1584413	Hip Support Pad, medium (230x100 mm)		1
5	SP1584412	Hip Support Pad, small (140x100 mm)		1

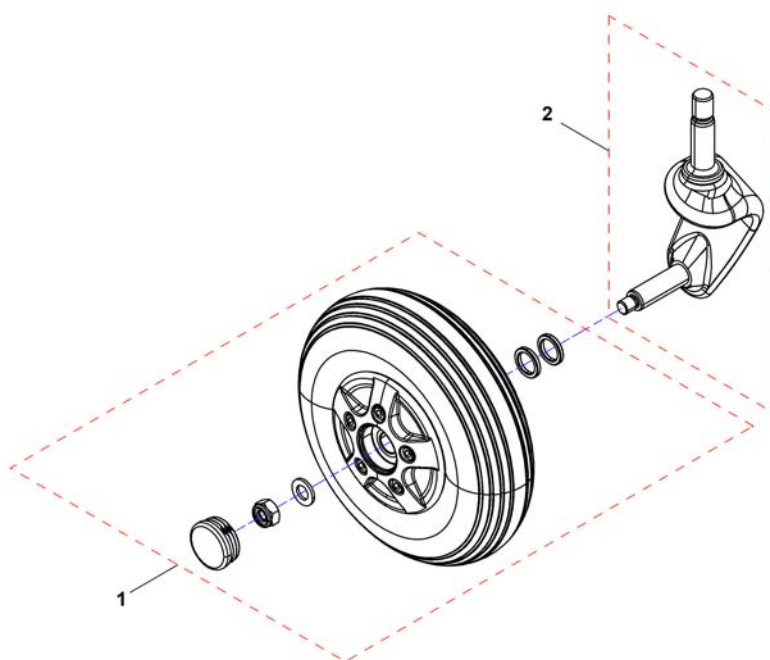

Pos.	Part number	Description	Valid from → until	Qty
1	SP1162534	Tire, Urethan 200mmx60mm, cpl.		1
2	SP1162570	Fork RH		1
2	SP1162571	Fork LH		1


Drive wheels 12.5"

1541673X.iso

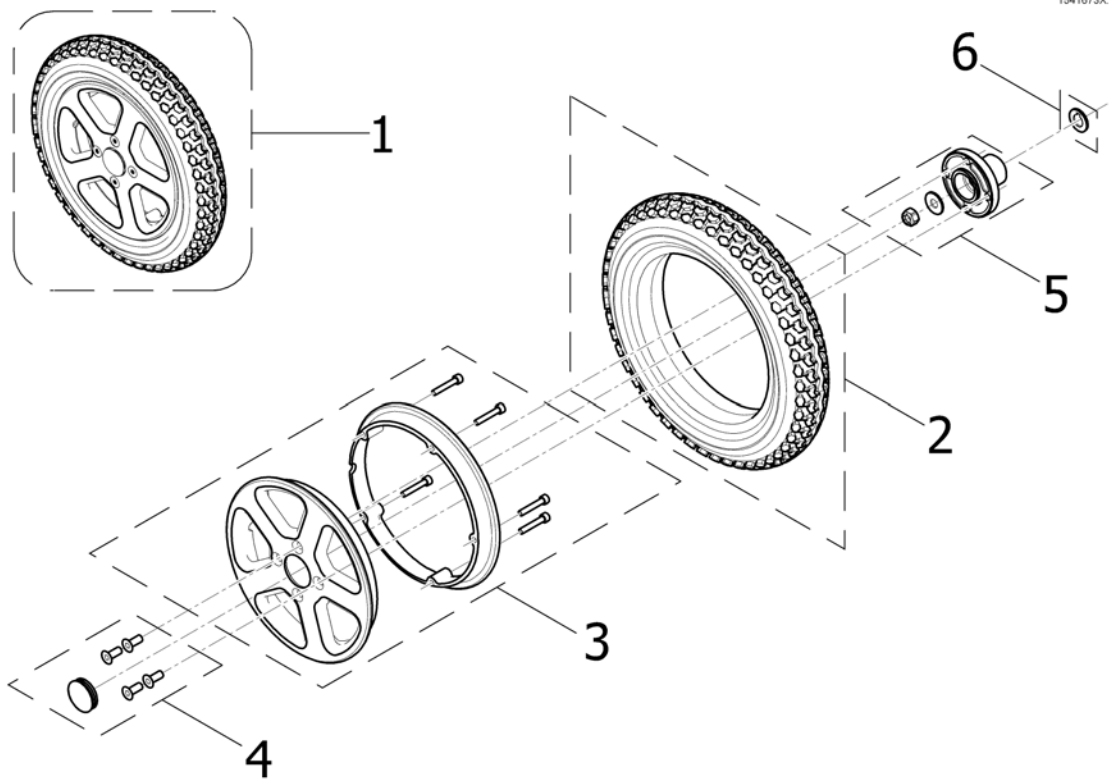

Pos.	Part number	Description	Valid from → until	Qty
1	1431291	Wheel Assy 12.5 x 2.25"		1
2	23009262	Tire 12.5" x 2.25" Puncture Proof		1
3	1440295	Rim 12.5" complete		1
4	SP1541741	Fastening-Kit		1
5	SP1542993	Hub, 4-hole		1
6	1423164-0002	Washer 19 mm with cutout for feather key		1

Drive wheels 14"

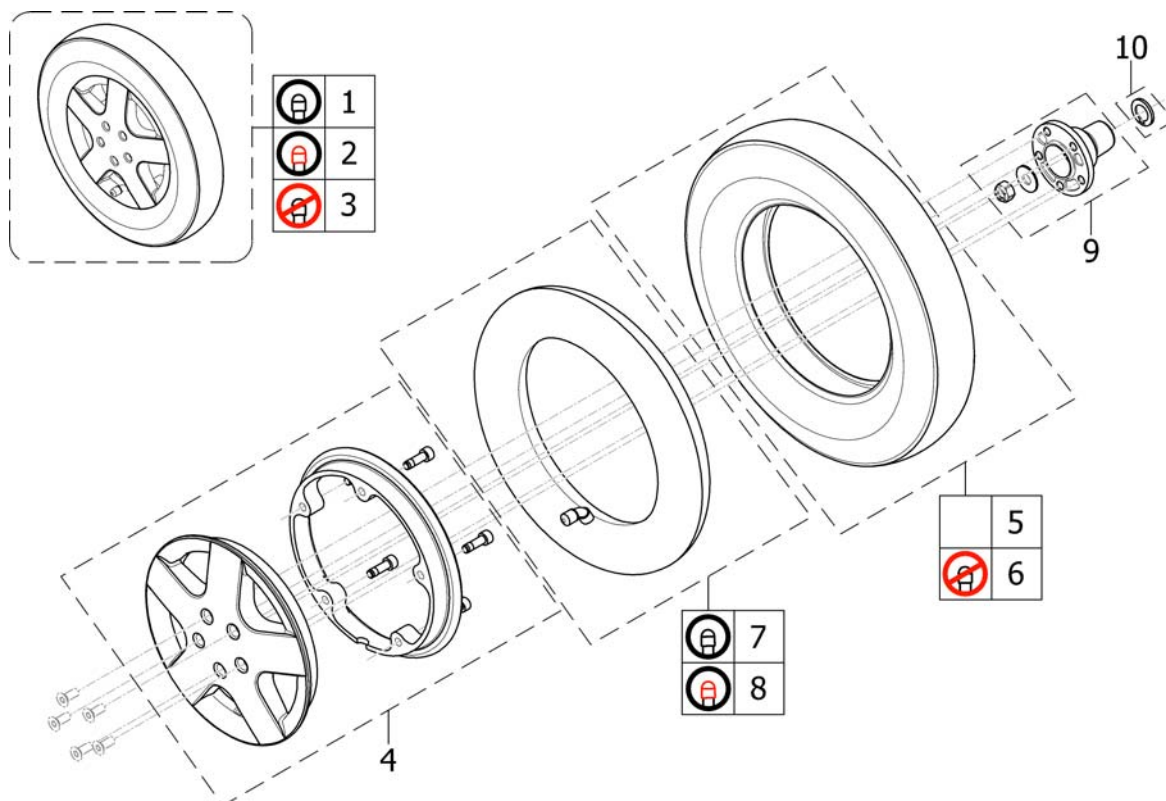

Pos.	Part number	Description	Valid from → until	Qty
1	SP1532414	Wheel 14 x 3.00" pneumatic, silver, complete		1
2	SP1532415	Wheel 14 x 3.00" puncture protected, silver, complete		1
3	SP1532416	Wheel 14 x 3.00" puncture proof, silver, complete		1
4	SP1541373	Rim Kit, complete		1
5	SP1522940	Tire 14 x 3.00"		1
6	SP1141342	Tire 14 x 3.00" puncture proof		1
7	SP1039651	Tube 14"		1
8	SP1541374	Tube 14" puncture protected		1
9	SP1542992	Hub, 5-hole		1
10	1423164-0002	Washer 19 mm with cutout for feather key		1

11/25/2004 1:00 PM

Overview Propulsion SSD


Pos.	Part number	Description	Valid from → until	Qty
18	SP1568148	Propulsion cpl. RH, Gearbox i = 23:1, Motor, 2-pole, 250W, slow, 600mm cable		1
19	SP1568149	Propulsion cpl. LH, Gearbox i = 18:1, Motor, 2-pole, 250W, fast, 600mm cable		1
20	SP1568150	Propulsion cpl. RH, Gearbox i = 18:1, Motor, 2-pole, 250W, fast, 600mm cable		1
21	SP1568151	Propulsion cpl. LH, Gearbox i = 23:1, Motor, 4-pole, 340W, slow, 760mm cable		1
22	SP1568152	Propulsion cpl. RH, Gearbox i = 23:1, Motor, 4-pole, 340W, slow, 760mm cable		1
23	SP1568153	Propulsion cpl. LH, Gearbox i = 18:1, Motor, 4-pole, 340W, fast, 760mm cable		1
24	SP1568208	Propulsion cpl. RH, Gearbox i = 18:1, Motor, 4-pole, 340W, fast, 760mm cable		1
25	SP1602974	Propulsion cpl. LH, Gearbox i = 23:1, Motor 4-pole 340W, slow, 760mm cable (incl. declutching lever)		1
26	SP1602975	Propulsion cpl. RH, Gearbox i = 23:1, Motor 4-pole 340W, slow, 760mm cable (incl. declutching lever)		1
27	SP1541366	Propulsion cpl. LH, Gearbox i = 18:1, Motor, 4-pole, 340W, fast, 760mm cable <i>S/N 061122610151 =></i>		1
28	SP1541365	Propulsion cpl. RH, Gearbox i = 18:1, Motor, 4-pole, 340W, fast, 760mm cable <i>S/N 061122610151 =></i>		1
29	SP1170307	Propulsion cpl. LH, Gearbox i = 23:1, Motor, 4-pole, 340W, fast, 760mm cable (incl. declutching lever)		1
30	SP1170306	Propulsion cpl. RH, Gearbox i = 23:1, Motor, 4-pole, 340W, fast, 760mm cable (incl. declutching lever)		1
31	SP1163413	Gearbox, LH, i = 18:1		1
32	SP1163412	Gearbox, RH, i = 18:1		1
33	SP1163415	Gearbox, LH, i = 23:1		1
34	SP1163414	Gearbox, RH, i = 23:1		1
35	SP1542953	Clamp band with clutch		1
36	SP1163099	Motor, 2-pole, 250W, fast, 600mm cable, ACS		1
37	SP1164046	Motor, 2-pole, 250W, slow, 600mm cable, ACS		1
38	SP1165863	Motor, 4-pole, 340W, fast, 760mm cable, ACS		1
39	SP1165865	Motor, 4-pole, 340W, slow, 760mm cable, ACS		1
40	SP1544840	Motor, 2-pole, 250W, fast, 600mm cable, VR2		1
41	SP1544796	Motor, 2-pole, 250W, slow, 600mm cable, R-Net		1
42	SP1544841	Motor, 2-pole, 250W, fast, 600mm cable, R-Net		1
43	SP1544795	Motor, 2-pole, 250W, slow, 600mm cable, VR2		1
45	SP1540358	Kit, brushes with caps, 2-pole (Set of 2)		1
46	SP1522967	Kit, brushes with caps, 4-pole (Set of 4)		1
47	SP1536022	Propulsion cpl. LH, Gearbox i = 23:1, Motor, 4-pole, 340W, slow, 760mm cable (incl. declutching knob) <i>S/N 061122812868 =></i>		1
48	SP1536021	Propulsion cpl. RH, Gearbox i = 23:1, Motor, 4-pole, 340W, slow, 760mm cable (incl. declutching knob) <i>S/N 061122812868 =></i>		1
49	SP1602976	Propulsion cpl. LH, Gearbox i = 18:1, Motor 4-pole 340W, fast, 760mm cable (incl. declutching lever)		1
50	SP1602998	Propulsion cpl. RH, Gearbox i = 18:1, Motor 4-pole 340W, fast, 760mm cable (incl. declutching lever)		1
51	SP1579595	Declutching knob SSD RH/LH (1 pc) <i>Single part. Consider to buy another one for the other side if required. - Replaces SP1168668, SP1168669 & SP1169005</i>		1
52	SP1168667	Declutching lever, cpl. - LH		1
53	SP1168666	Declutching lever, cpl. - RH		1
54	SP1552782	Propulsion cpl. LH, Gearbox i = 23:1, Motor 4-pole, 340W, slow, 760mm cable, ACS		1
55	SP1552783	Propulsion cpl. RH, Gearbox i = 23:1, Motor 4-pole, 340W, slow, 760mm cable, ACS		1

Overview Propulsion SSD


Pos.	Part number	Description	Valid from → until	Qty
56	SP1552872	Propulsion cpl. LH, Gearbox i = 18:1, Motor 4-pole, 340W, fast, 760mm cable, ACS		1
57	SP1552873	Propulsion cpl. RH, Gearbox i = 18:1, Motor 4-pole, 340W, fast, 760mm cable, ACS		1
58	SP1552832	Propulsion cpl. LH, Gearbox i = 23:1, Motor 4-pole, 340W, slow, 760mm cable, R-Net		1
59	SP1552833	Propulsion cpl. RH, Gearbox i = 23:1, Motor 4-pole, 340W, slow, 760mm cable, R-Net		1
60	SP1552836	Propulsion cpl. LH, Gearbox i = 18:1, Motor 4-pole, 340W, fast, 760mm cable, R-Net		1
61	SP1552837	Propulsion cpl. RH, Gearbox i = 18:1, Motor 4-pole, 340W, fast, 760mm cable, R-Net		1
62	SP1562498	Motor, 4-Pol, 340W, slow, 760mm cable, R-Net		1
63	SP1562499	Motor, 4-Pol, 340W, fast, 760mm cabel, R-Net		1

Swingarms

1168373X.dwg

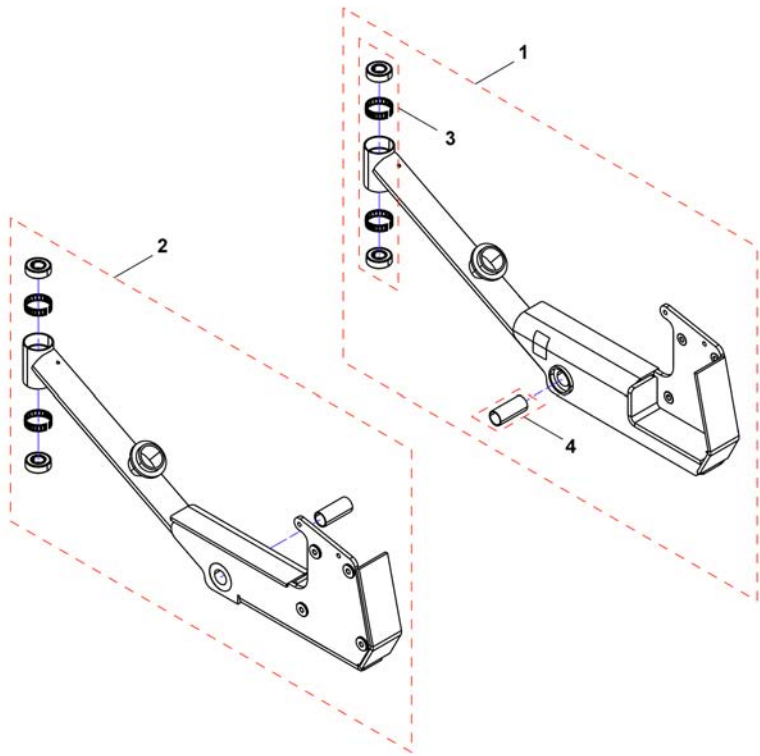

Pos.	Part number	Description	Valid from → until	Qty
1	SP1168373	Swingarm, RH, cpl.		1
2	SP1168374	Swingarm, LH, cpl.		1
3	SP1541806	Bearing-Kit		1
4	SP1165591	Sleeve		1

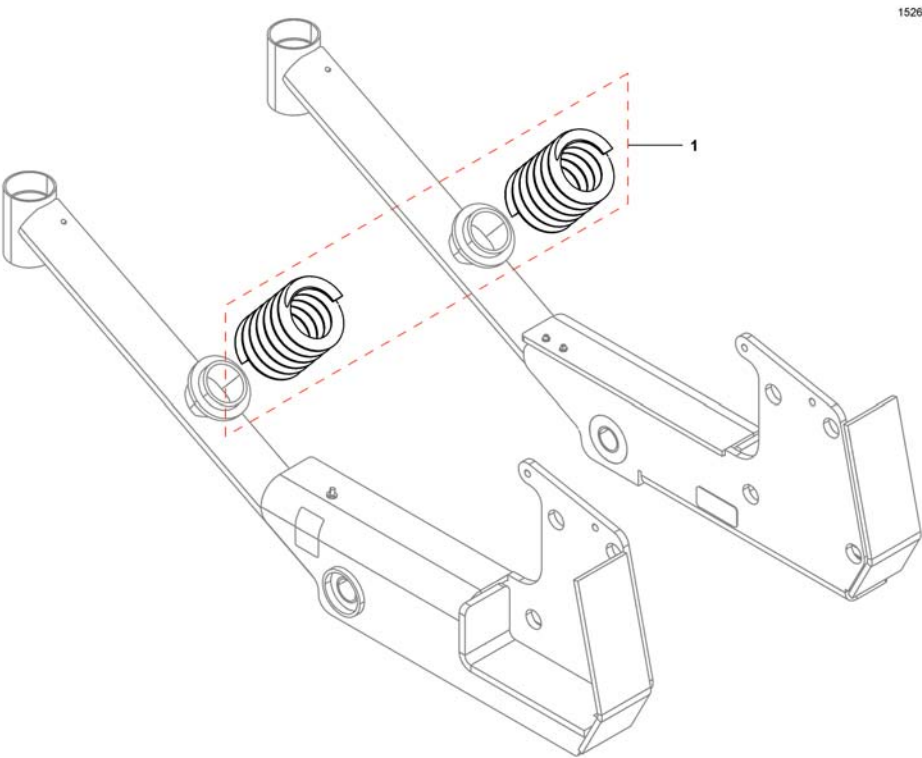


1526857X.dwg

Pos.	Part number	Description	Valid from → until	Qty
1	SP1536857	Springs, hard (pair) <i>includes compensating discs for 12,5" wheels</i>		1
1	SP1168897	Springs, soft (pair) <i>for chairs without lifter only</i>		1

Electronic

FDX75 line

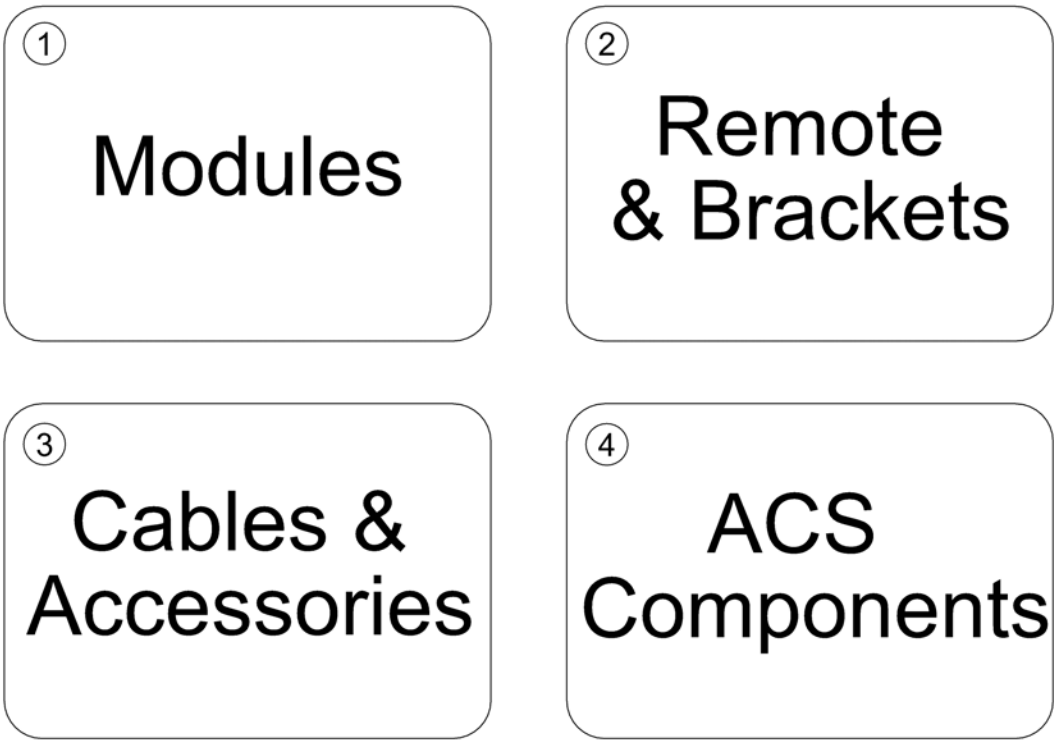


Pos.	Part number	Description	Valid from → until	Qty
1	-	ACS Electronic Modules		1
2	-	ACS Remotes & Brackets		1
3	-	ACS Cables & Accessories		1
4	-	ACS Components		1

ACS Electronic Modules

Pos.	Part number	Description	Valid from → until	Qty
-		Powermodule & Gyroskop		1
-		Powermodules		1
-		Holder ACS2 Powermodule		1
-		ACT for ACS 2		1
-		Holder ACT		1
-		Lightboard		1

Powermodule & Gyroskop

1537975X.dwg

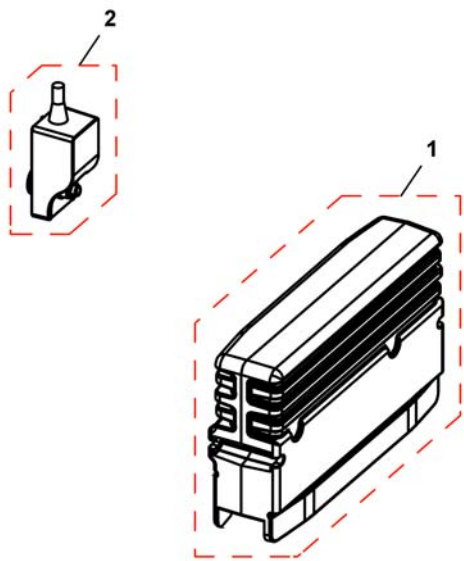

Pos.	Part number	Description	Valid from → until	Qty
-	SP1555702	Gasket kit for ACS2, Powermodule and ACT		1
1	SP1537975	Powermodule, ACS2, 90A, for light and G-Trac, DX2-PMA90LG <i>Lock washer between module and bracket!</i>		1
2	SP1537848	DX2 G-Trac-sensor		1

Powermodules


Powermodule 80

	Dynamics											ACS 2				Penny & Giles				R-Net
	Shark					ACS						ACS 2				Pilot +			VR2	
	DK-PMB01	DK-PMB21	DK-PMB28	DK-PMB50	DK-PMC28	DX-PMB-AS	DX-PMB-2	DX-PMB2-SAS	DX-GB-AS	DX-GB-ASG	DX-GB-AS II	DX2-PMA70	DX2-PMA70L	DX2-PMA90L	DX2-PMA90LG	Pilot+ 50-xM 50A	VR2 60	VR2 60-A2		
Bora / SpectraXTR	2	3	8									12	13	14	15					
Dragon	2	3					28										25	26		
Dragon Vertic	2	3					29					12	13	14	15					
FDX								7												
G40								7												
G50																				
Mirage	2	3														24	25	26		
Mistral	2	3				NLA														
Pronto M61	2	3																		
Spectra Blitz	2	3				NLA										24	25	26	27	
Spectra Comfort	2	3				NLA						12	13			24	25	26	27	
Spectra Plus	2	3				NLA						12	13			24	25	26	27	
Spectra Seat Riser	2	3				NLA										24	25	26	27	
Storm ³						NLA	30													
Storm ³ GB									9											
Storm ³ Vertic							31													
Storm4															14	15				
Storm4 MAX															14	15				
Stream			8	6																
TDX-SP					4															
Tiger		3					28					12	13							
Topan	2	3					28													
Tornado	2	3					NLA													
Typhoon							NLA													
Typhoon GB									9		9									
Typhoon II									9											
Zephyr										10	9									

Pos.	Part number	Description	Valid from → until	Qty
NLA	-	No longer available.		0
-	SP1555702	Gasket kit for ACS2, Powermodule and ACT		1
2	1433332	Powermodule, Shark, 75A, drive only, DK-PMB01		1
3	1448744	Powermodule, Shark, 75A, 2 actuators, DK-PMB21		1
4	1136898	Powermodule, Shark, 90A, 2 actuators, DK-PMC28		1
6	SP1537235	Powermodule, Shark, DK-PMB50		1
7	1451448	Powermodule, ACS, 160A, DX-PMB2-SAS		1
8	SP1577053	Powermodule, Shark, 75A, 2 Actuators, DK-PMB28 [0977]		1
9	1430150	Powermodule for GB-motors, ACS, 100A, DX-GB AS		1
10	AT.1496444	Powermodule for GB-motors, ACS, 100A, (only Zephyr), DX-GB-ASG		1
12	SP1528673	Powermodule, ACS2, 70A, DX2-PMA70 <i>Lock washer between module and bracket!</i>		1
13	SP1523433	Powermodule, ACS2, 70A, with light, DX2-PMA70L <i>Lock washer between module and bracket!</i>		1
14	SP1524244	Powermodule, ACS2, 90A, with light, DX2-PMA90L <i>Lock washer between module and bracket!</i>		1
15	SP1537975	Powermodule, ACS2, 90A, for light and G-Trac, DX2-PMA90LG <i>Lock washer between module and bracket!</i>		1
24	30000265	Powermodule, Pilot +, 50A, Pilot+ 50-xM		1
25	SP1504056	Powermodule, VR2, 60A, only drive, VR2 60		1
26	SP1510131	Powermodule, VR2, 60A, 2 actuators, VR2 60-A2		1
27	SP1532445	Powermodule, R-net, 80A, 2 actuators, R-net EL PM90		1
28	-	Upgrade-Kit 80A ACS => ACS2, Dragon / Tiger / Topan		1
29	-	Upgrade-Kit 80A ACS => ACS2, Dragon Vertic		1
30	-	Retrofit-Kit 80A ACS => ACS2		1
31	-	Upgrade-Kit 80A ACS => ACS2, Storm ³ Vertic		1

Upgrade-Kit 80A ACS => ACS2, Dragon / Tiger / Topan


1570293.iss

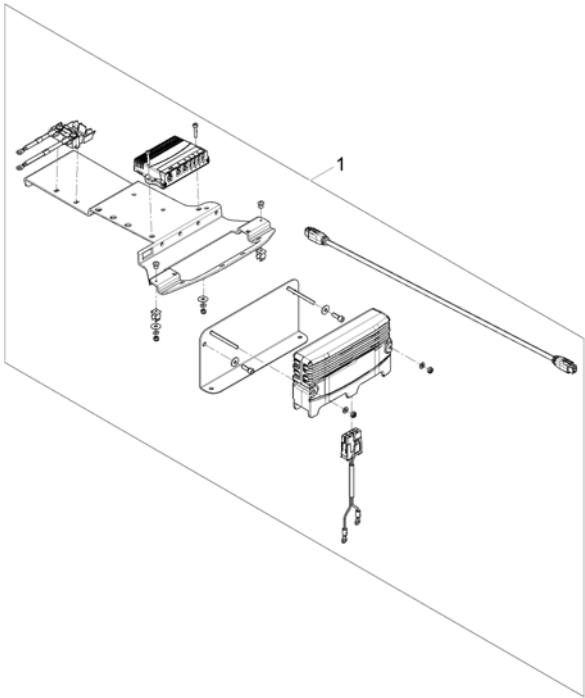

Pos.	Part number	Description	Valid from → until	Qty
1	SP1570293	Upgrade-Kit ACS 80A ==> ACS2 <i>Driver program to be updated. Please contact IVCR Service / Lock wascher between module and bracket!</i>		1

Upgrade-Kit 80A ACS => ACS2, Dragon Vertic


1571619.00

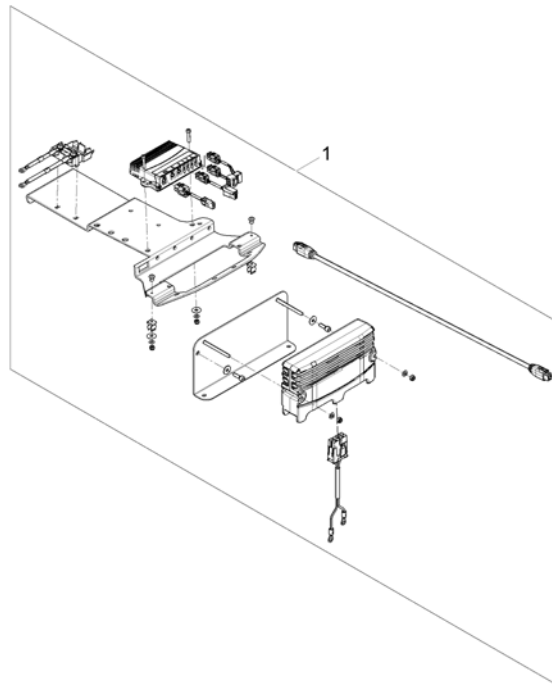

Pos.	Part number	Description	Valid from → until	Qty
1	SP1570367	Upgrade-Kit DragonVertic ACS 80A ==> ACS2 <i>Driver program to be updated. Please contact IVCR service / Lock washer between module and bracket!</i>		1

Retrofit-Kit 80A ACS => ACS2


1570376 Rev

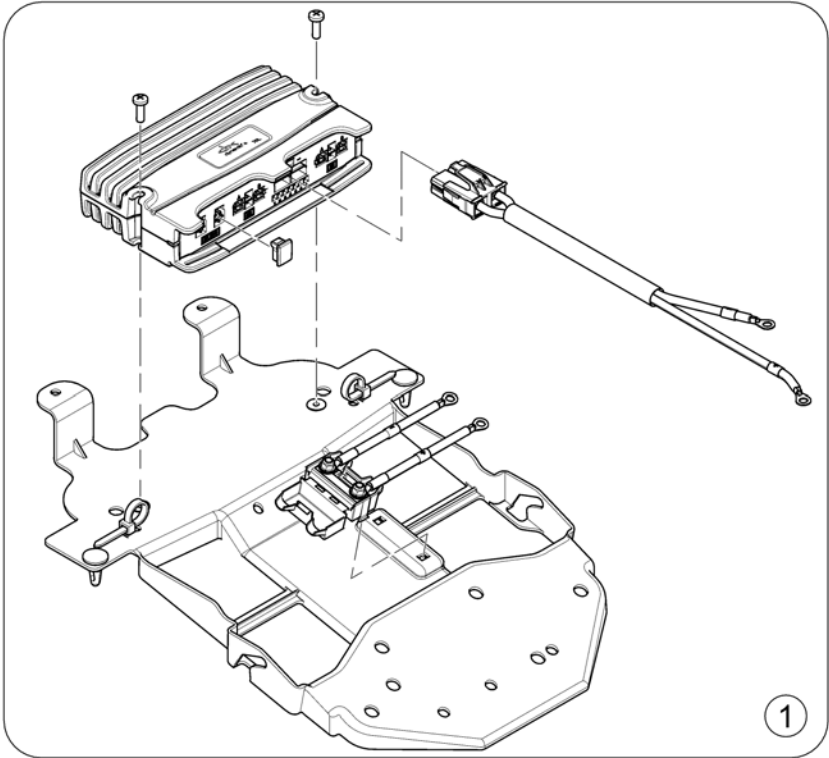


Pos.	Part number	Description	Valid from → until	Qty
1	SP1570290	Retrofit-Kit 80A ACS => ACS2 <i>Driver program to be updated. Please contact IVCR Service</i>		1

Upgrade-Kit 80A ACS => ACS2, Storm³ Vertic

1570369.002

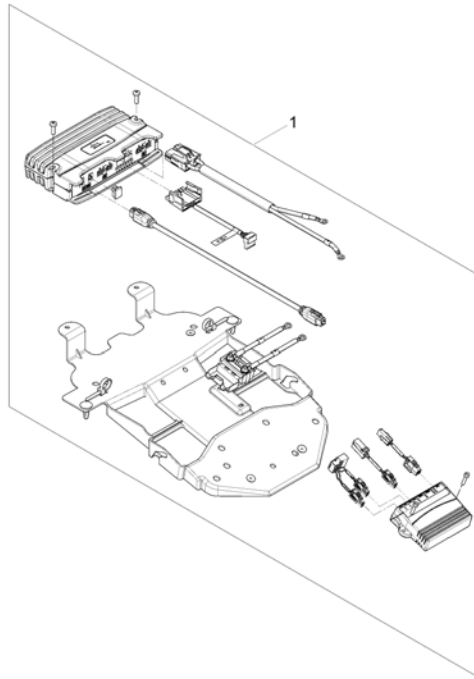


Pos.	Part number	Description	Valid from → until	Qty
1	SP1570369	Upgrade-Kit ACS 80A ==> ACS2, Storm ³ Vertic <i>Driver program to be updated. Please contact IVCR Service</i>		1

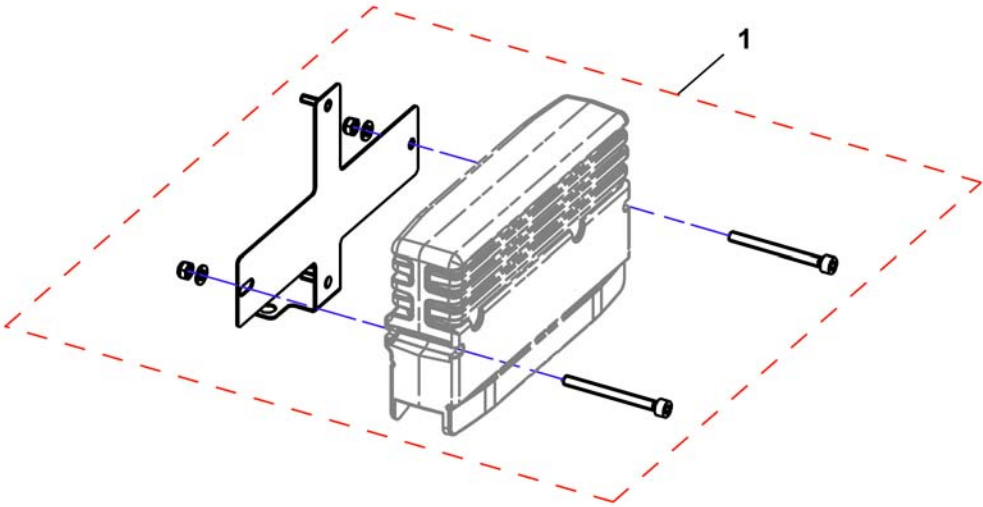

Pos.	Part number	Description	Valid from → until	Qty
1	SP1541742	Holder Powermodule and ACT <i>Lock washer between module and bracket!</i>		1

ACT for ACS 2


1528542X.iso

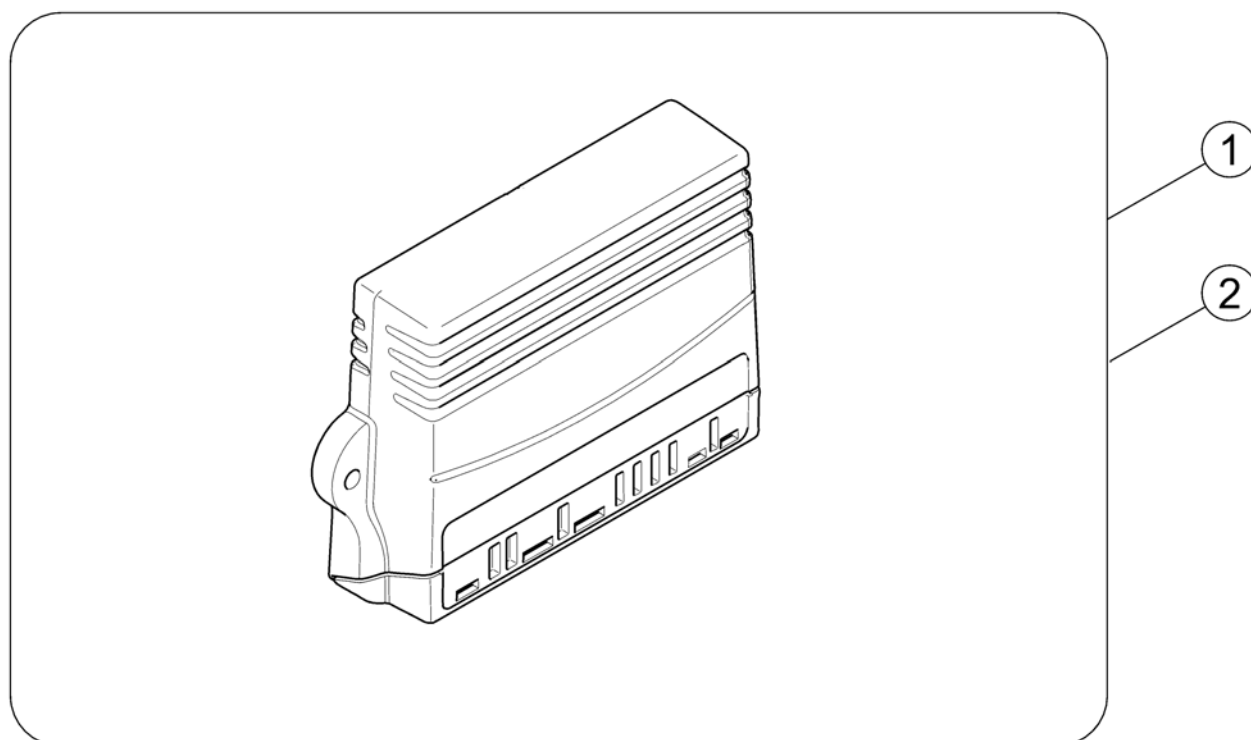


Pos.	Part number	Description	Valid from → until	Qty
1	SP1528542	ACT2 for ACS2, cpl.		1
2	SP1529382	ACT4 for ACS2, cpl.		1

Holder ACT

1535797X.dwg

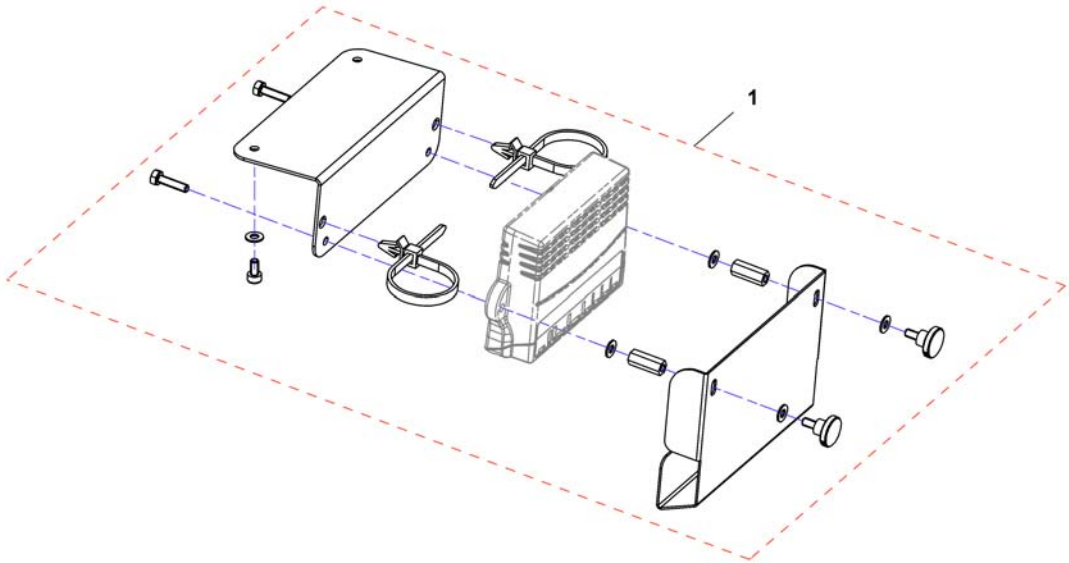


Pos.	Part number	Description	Valid from → until	Qty
1	SP1541743	Holder ACT		1

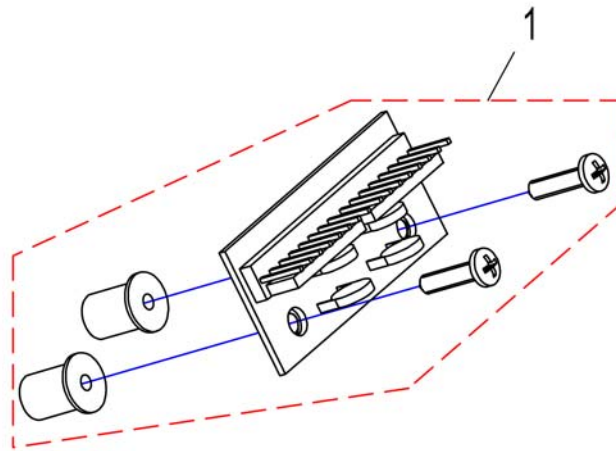


Pos.	Part number	Description	Valid from → until	Qty
1	SP1528550	Lightboard for all elektronik systems		1


ACS Remotes & Brackets

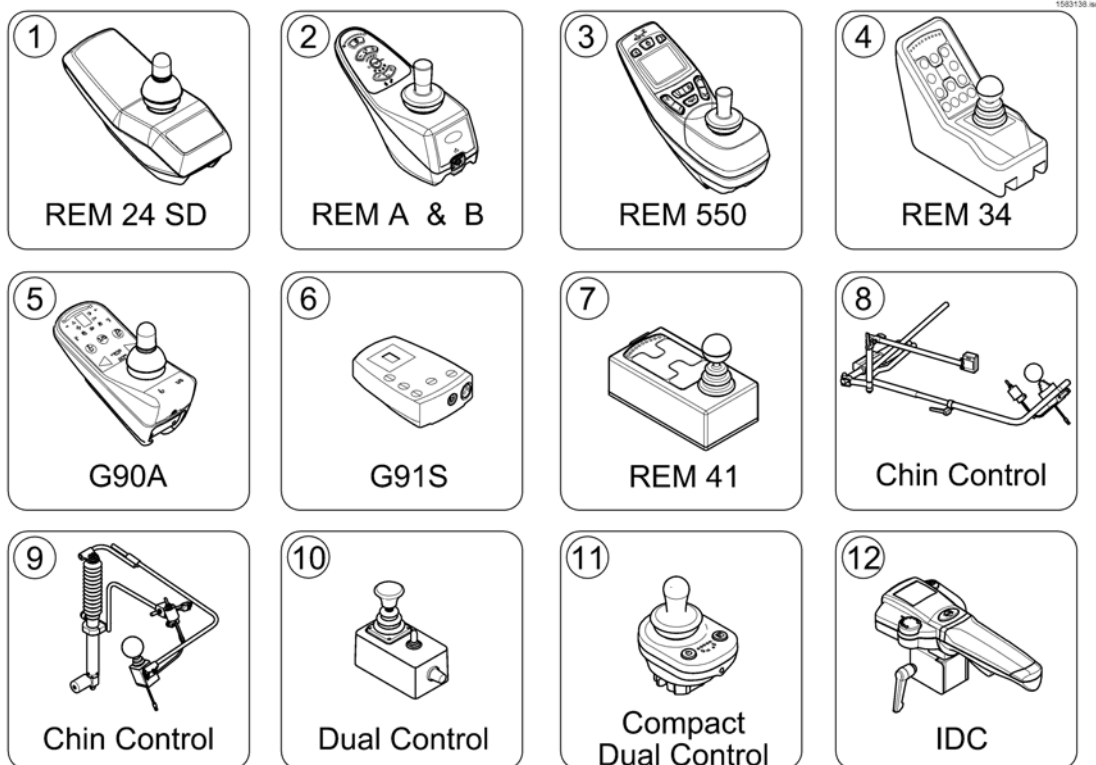


Pos.	Part number	Description	Valid from → until	Qty
1	-	REM 24 SD with brackets [1601/1602]		1
2	-	REM A & B with brackets [1638/1639/1643/1644]		1
3	-	REM 550 with brackets [1617/1618]		1
4	-	REM 34 with brackets [1611]		1
5	-	G90A with brackets [1614/1616]		1
6	-	G91S with brackets [1615]		1
7	-	Centre control REM 41 [1607/1608]		1
8	-	Small Chin Control (old) with brackets [1609]		1
9	-	Chin Control (new) with mounting interface - Modulite Seat [1719]		1
10	-	Dual Control with brackets [1291/1292]	→ 2015-06-30	1
11	-	Compact Dual Control with brackets [1720]	2015-07-01 →	1
12	-	Intuitive Dual Control (IDC) with bracket [1645]		1

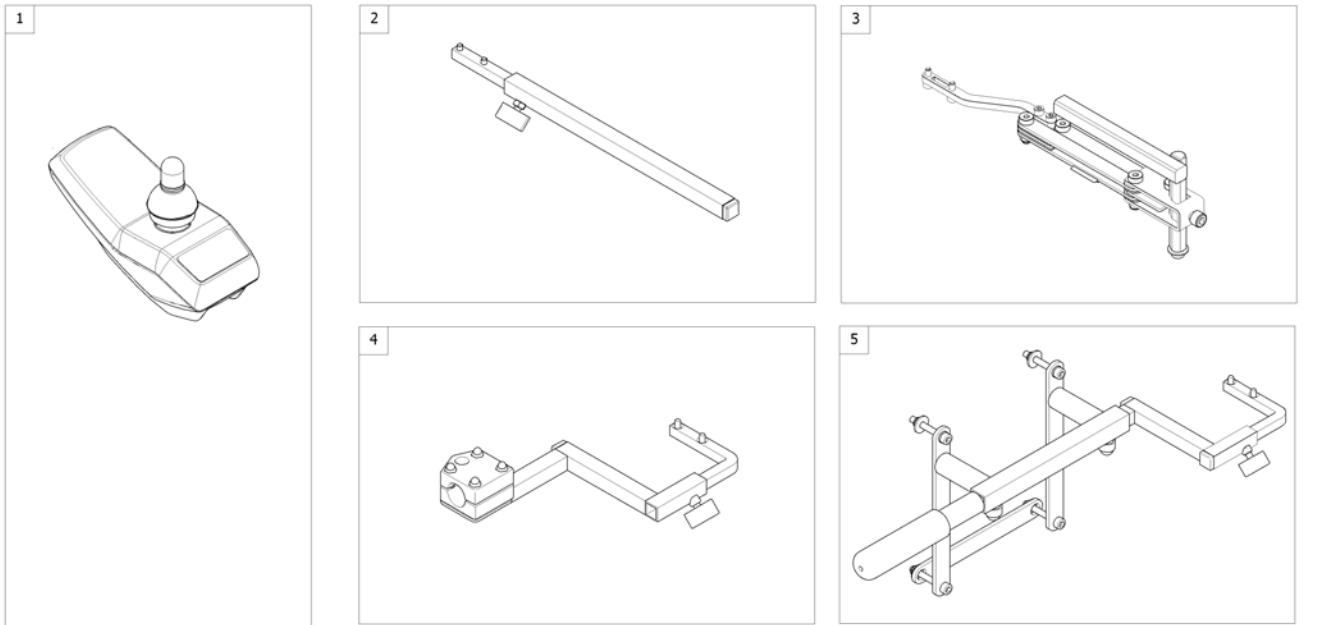

Pos.	Part number	Description	Valid from → until	Qty
1	-	Remote REM 24SD [1601/1602]		0
2	-	Remote Bracket Standard [1601+1207/1208]		1
3	-	Remote Bracket Swing away & height adjust. [1601+1707/1708]		1
4	-	Remote Bracket Attendant for Standard-/EasyAdapt-Seat [1602+1207/1208]		1
5	-	Remote Bracket Attendant for Recaro-Seat [1602+1207/1208]		1

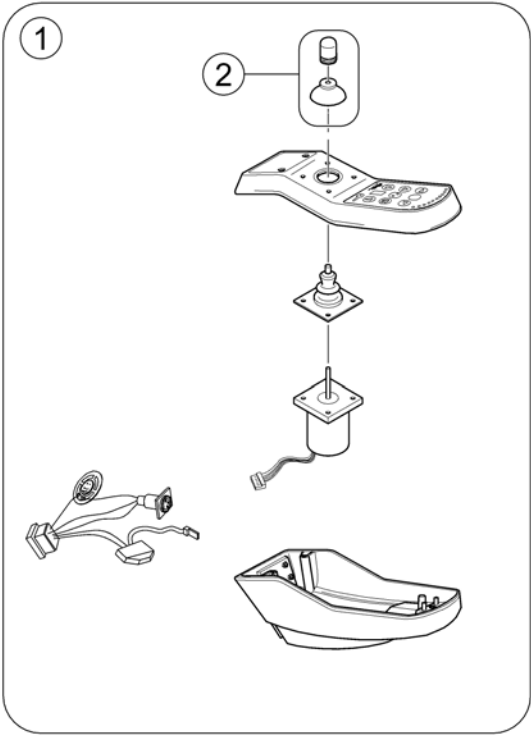


Pos.	Part number	Description	Valid from → until	Qty
1	1436541	Remote REM24SD		1
2	SPZ183229	Remote knob cpl.		1

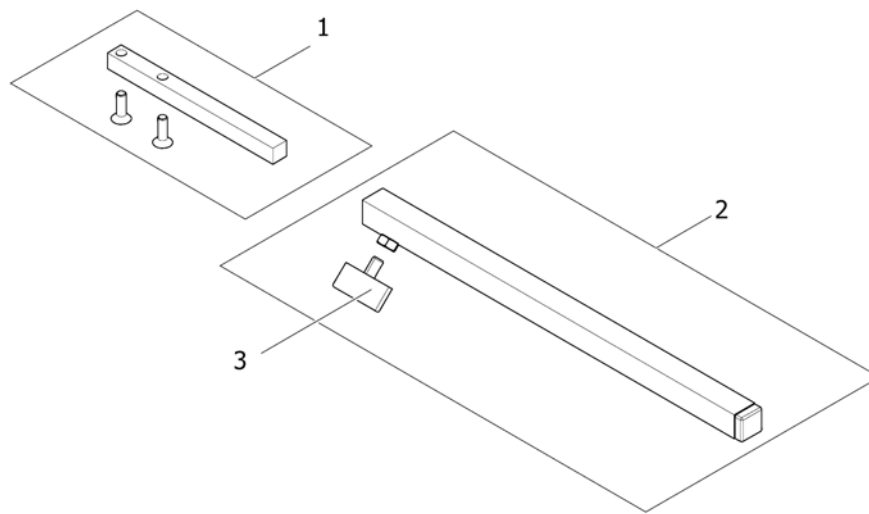

Pos.	Part number	Description	Valid from → until	Qty
1	SP1436542	Support, Standard		1
2	SPS183040	Remote Bracket, Standard		1
3	L31001	Wing Screw M6X10		1

Remote Bracket Swing away & height adjust.


1559852.iso

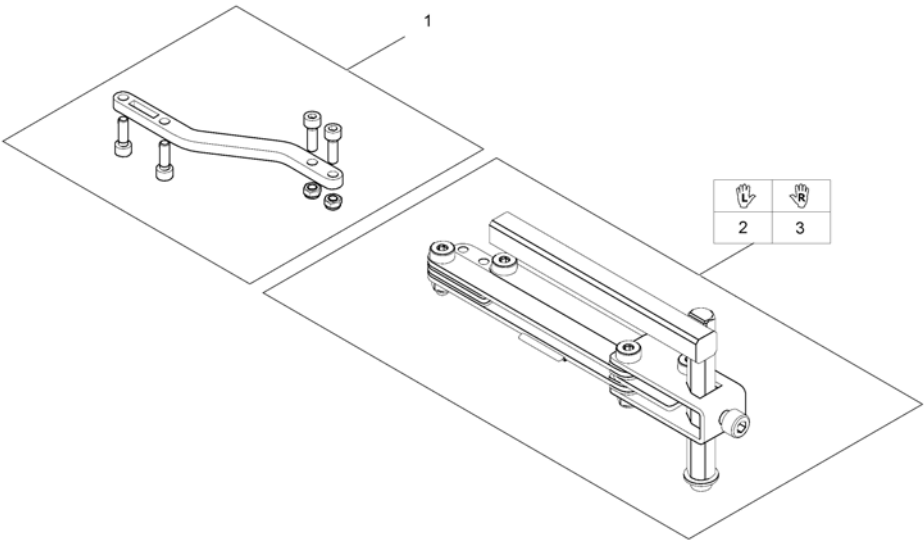

Pos.	Part number	Description	Valid from → until	Qty
1	SP1498222	Remote Support for Bracket height adjustable and swing away		1
2	SP1496249	Remote bracket, height adjustable & swing away, LH		1
3	SP1496248	Remote bracket, height adjustable & swing away, RH		1

Remote Bracket Attendant for Standard-/EasyAdapt-Seat


1560837.iso

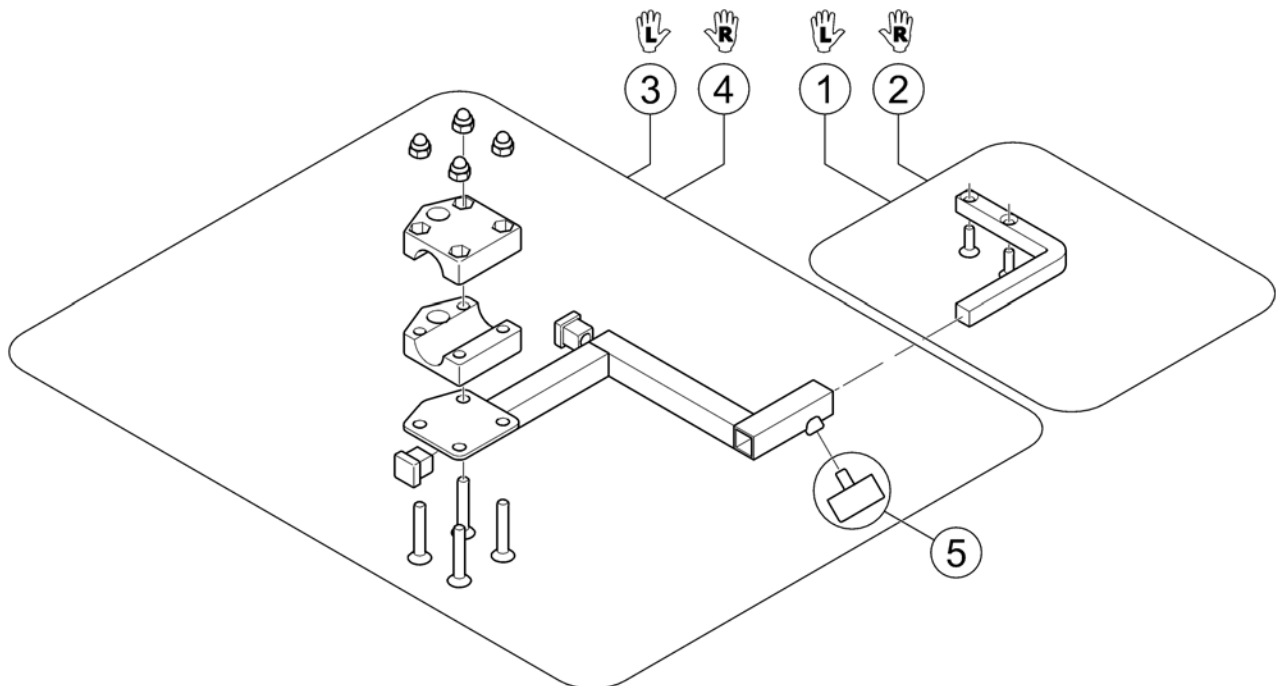


Pos.	Part number	Description	Valid from → until	Qty
1	1422806	Remote Support LH		1
2	1422810	Remote Support RH		1
3	SP1422937	Bracket for attendant control LH		1
4	SP1422936	Bracket for attendant control RH		1
5	L31001	Wing ScREW M6X10		1

Remote Bracket Attendant for Recaro-Seat

1563480.iso

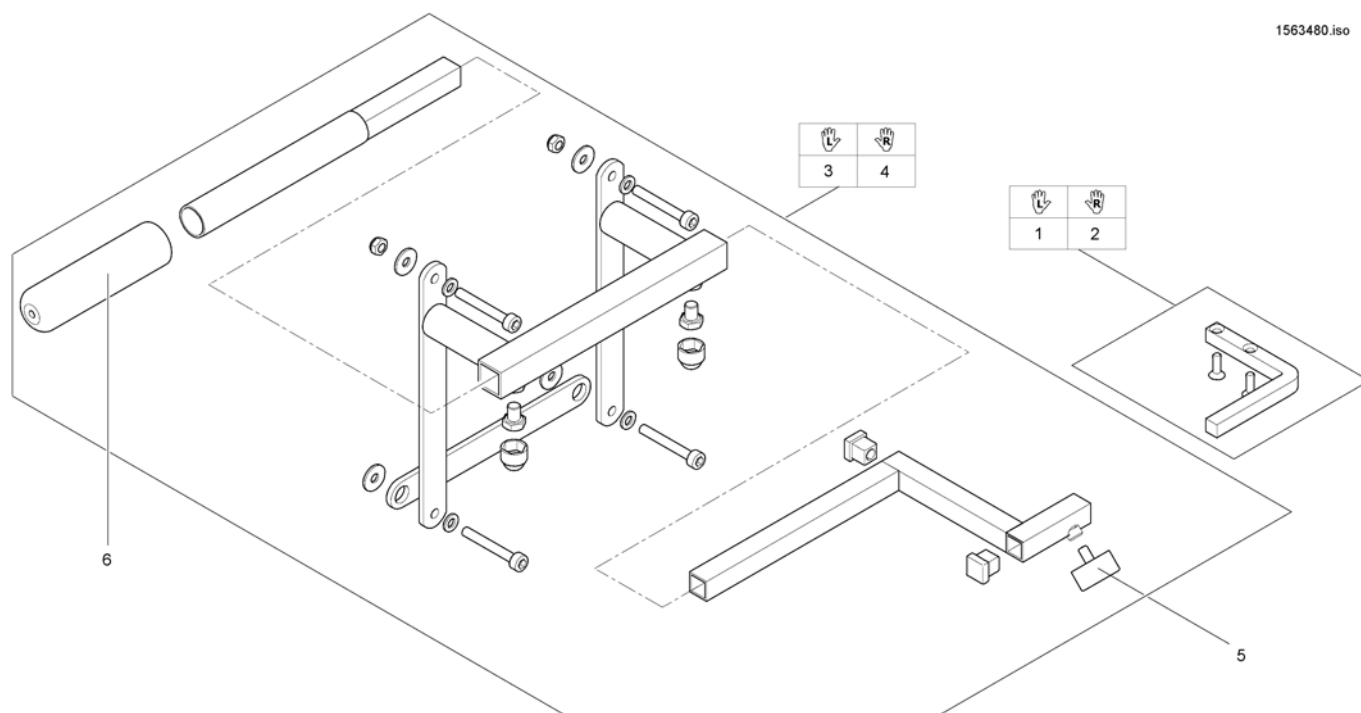


Pos.	Part number	Description	Valid from → until	Qty
1	1422806	Remote Support LH		1
2	1422810	Remote Support RH		1
3	SP1422814	Assebmly kit for dual control cpl., Recaro, LH		1
4	SP1422813	Assebmly kit for dual control cpl., Recaro, RH		1
5	L31001	Wing Scew M6X10		1
6	1435421	Handle plastic black Ø22 "Invacare"		1

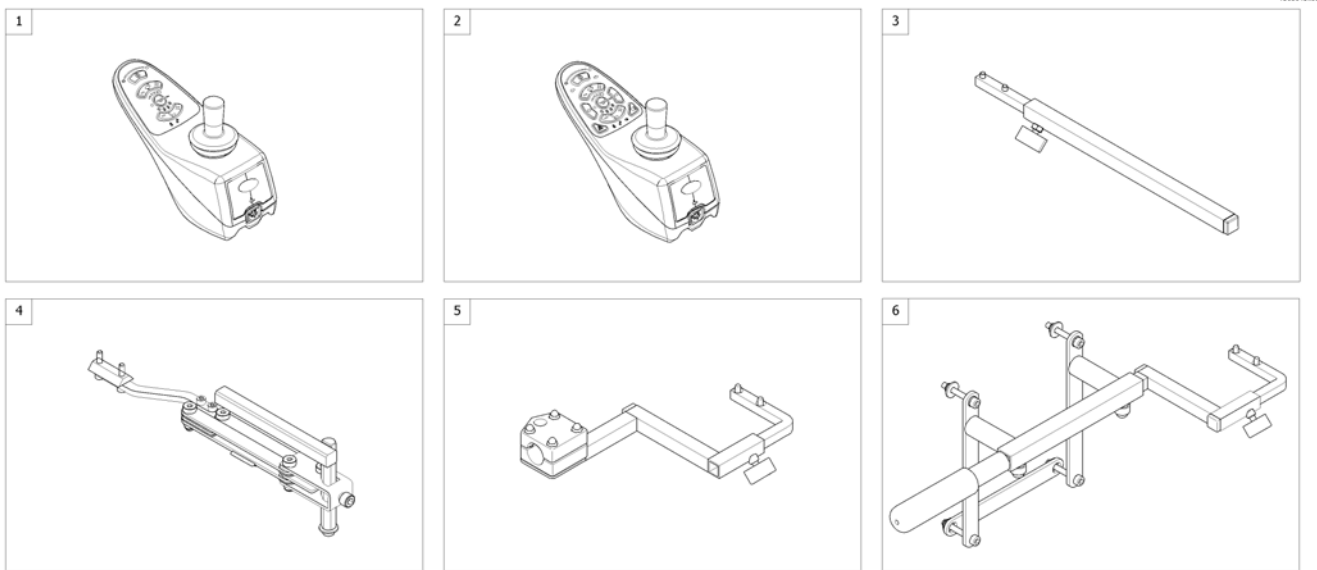


Pos.	Part number	Description	Valid from → until	Qty
1	SP1523440	Remote REM A [1638/1643]		1
2	SP1523445	Remote REM B [1639/1644]		1
3	-	Remote Bracket Standard [1638/1639+1207/1208]		1
4	-	Remote Bracket Swing away & height adjust. [1638/1639+1707/1708]		1
5	-	Remote Bracket Attendant for Standard-/EasyAdapt-Seat [1643/1644+1207/1208]		1
6	-	Remote Bracket Attendant for Recaro-Seat [1643/1644+1207/1208]		1

Remote Bracket Swing away & height adjust.


1559857.iso

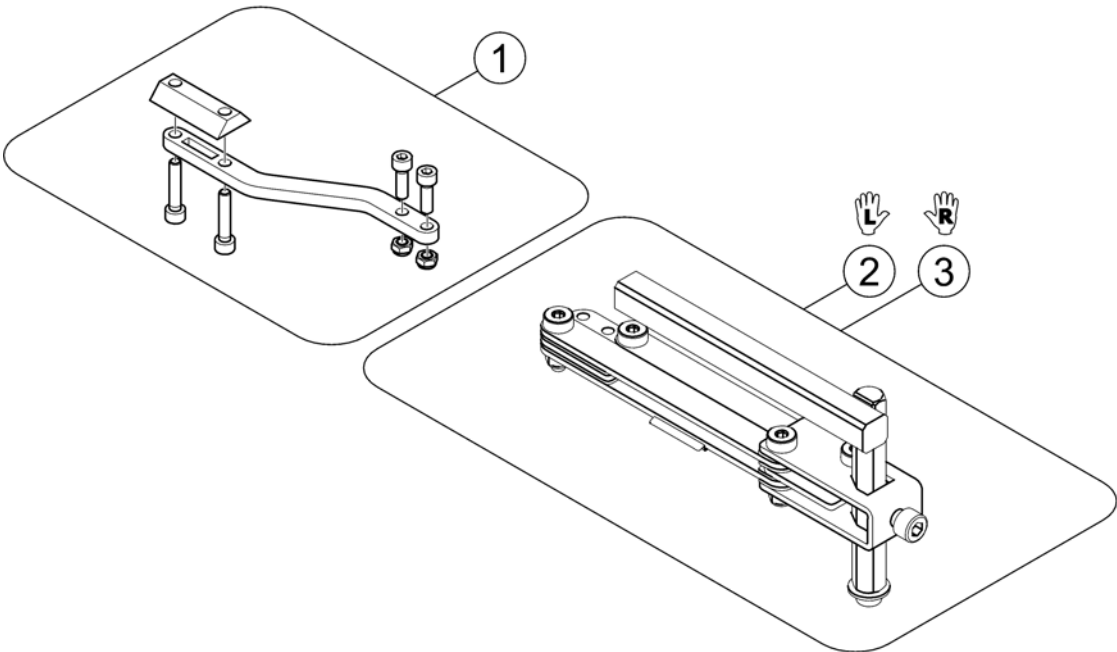


Pos.	Part number	Description	Valid from → until	Qty
1	SP1536180	Support for Remote Bracket, height adjustable & swing away		1
2	SP1496249	Remote bracket, height adjustable & swing away, LH		1
3	SP1496248	Remote bracket, height adjustable & swing away, RH		1

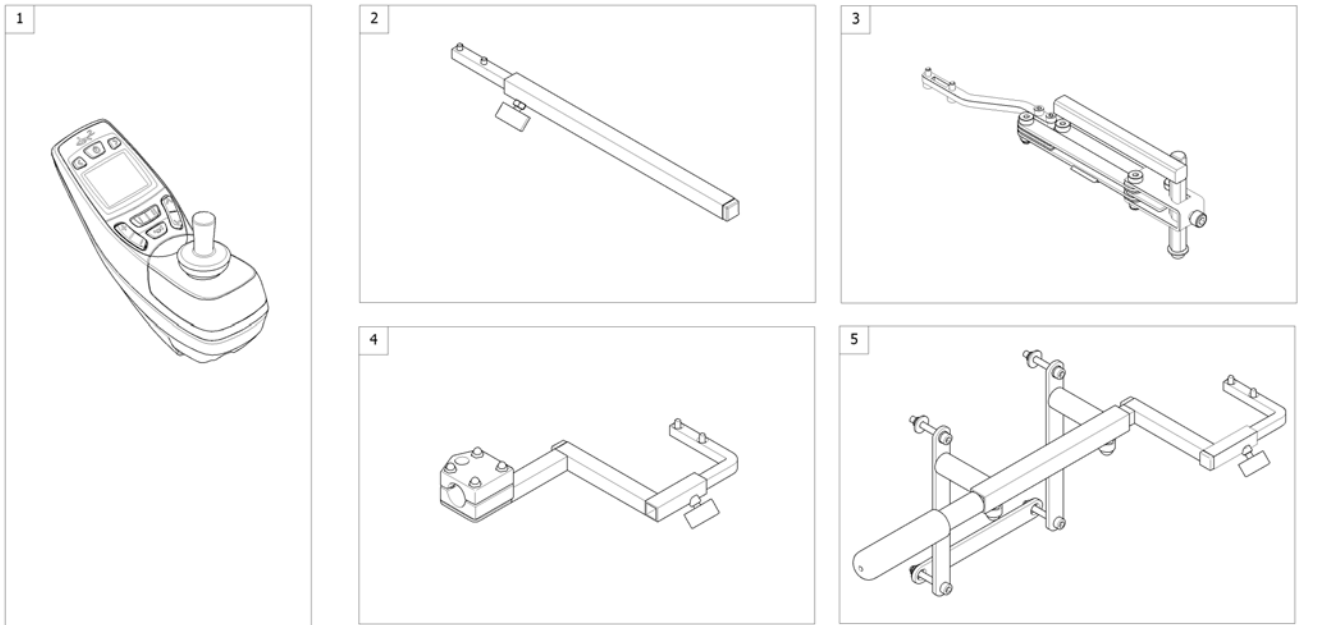

Pos.	Part number	Description	Valid from → until	Qty
1	SP1523444	Remote REM550 [1617/1618]		1
2	-	Remote Bracket Standard [1617+1207/1208]		1
3	-	Remote Bracket Swing away & height adjust. [1617+1707/1708]		1
4	-	Remote Bracket Attendant for Standard-/EasyAdapt-Seat [1618+1207/1208]		1
5	-	Remote Bracket Attendant for Recaro-Seat [1618+1207/1208]		1

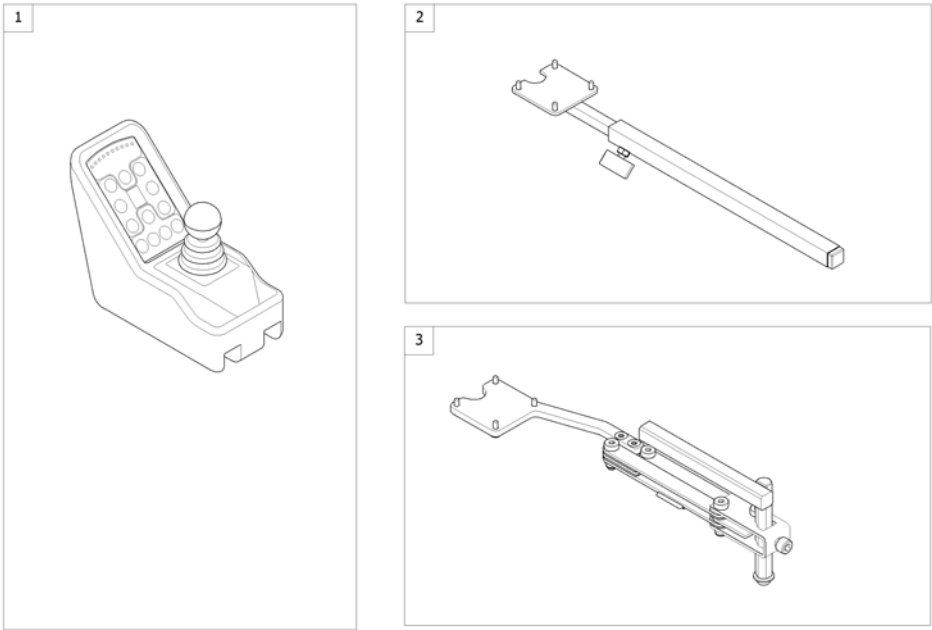


Pos.	Part number	Description	Valid from → until	Qty
1	SP1440677	Remote REM 34 [1611]		1
2	-	Remote Bracket Standard [1611+1207/1208]		1
3	-	Remote Bracket Swing away & height adjust. [1611+1707/1708]		1

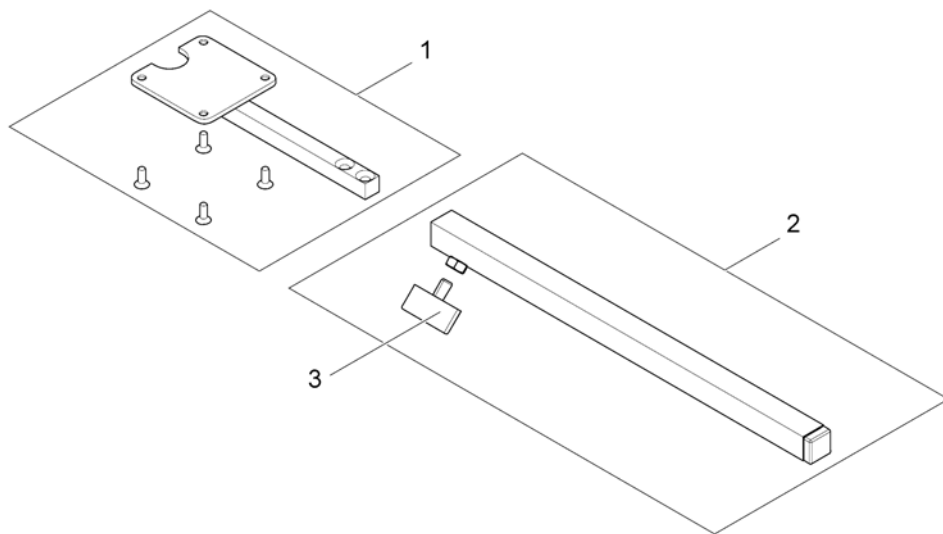


Pos.	Part number	Description	Valid from → until	Qty
1	SP1434921	Remote Support Standard		1
2	SPS183040	Remote Bracket, Standard		1
3	L31001	Wing ScREW M6X10		1

Remote Bracket Swing away & height adjust.

1560845.iso

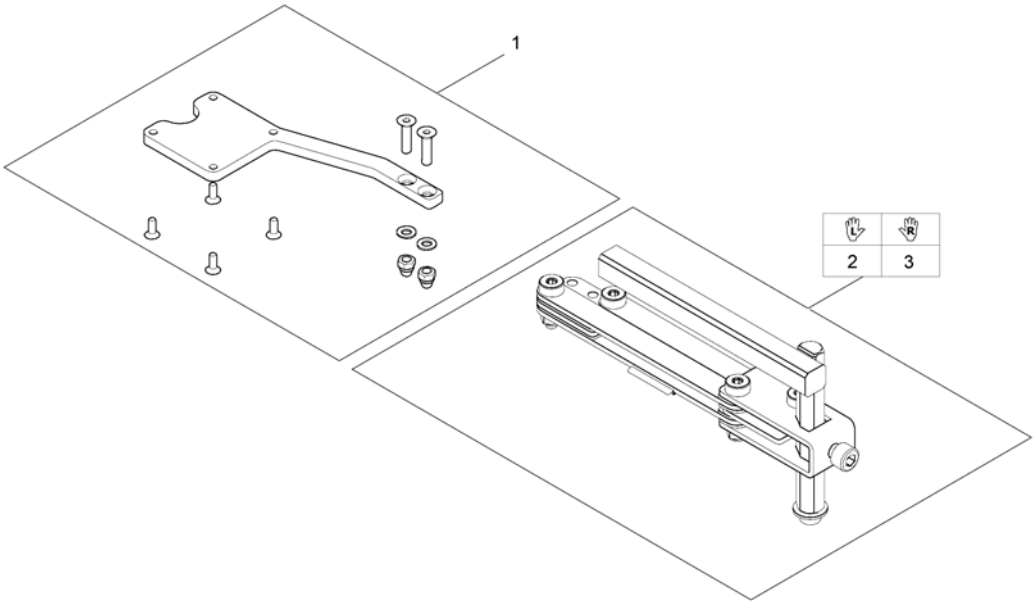


Pos.	Part number	Description	Valid from → until	Qty
1	SP1527125	Remote Support for Bracket height adjustable and swing away		1
2	SP1496249	Remote bracket, height adjustable & swing away, LH		1
3	SP1496248	Remote bracket, height adjustable & swing away, RH		1

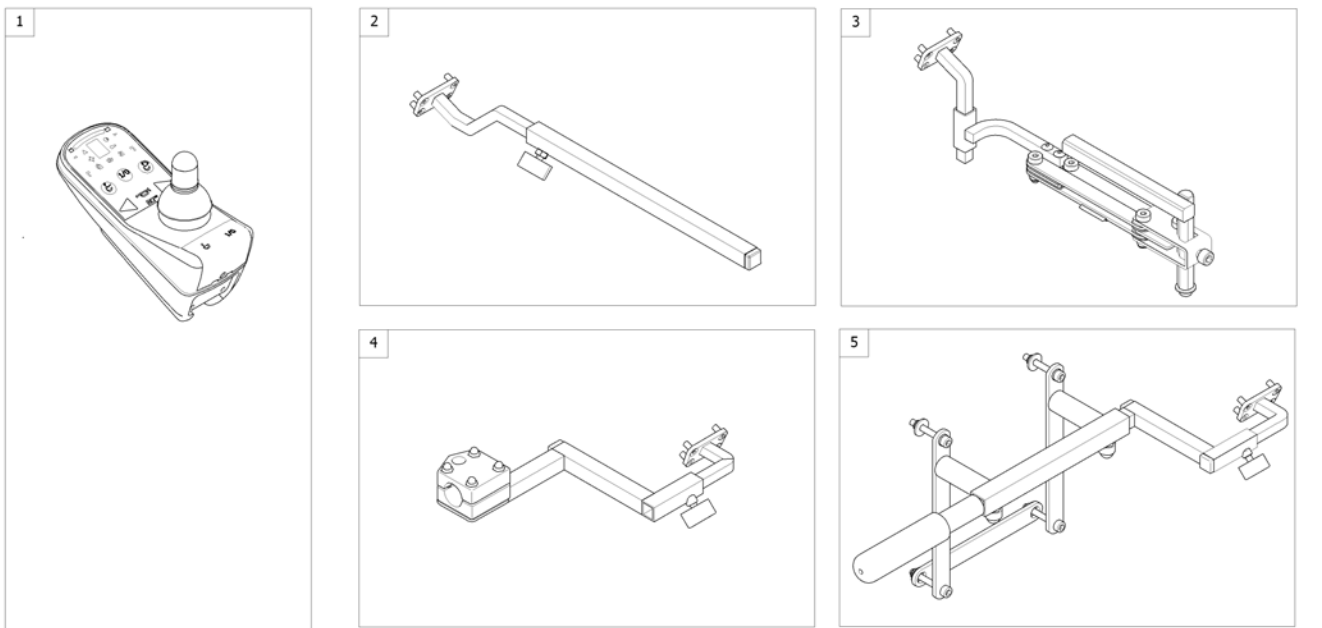

Pos.	Part number	Description	Valid from → until	Qty
1	SP1504720	Remote G90A [1614/1616]		1
2	-	Remote Bracket Standard [1614+1207/1208]		1
3	-	Remote Bracket Swing away & height adjust. [1614+1707/1708]		1
4	-	Remote Bracket Attendant for Standard-/EasyAdapt-Seat [1616+1207/1208]		1
5	-	Remote Bracket Attendant for Recaro-Seat [1616+1207/1208]		1

Remote Bracket Standard


1562939.iso

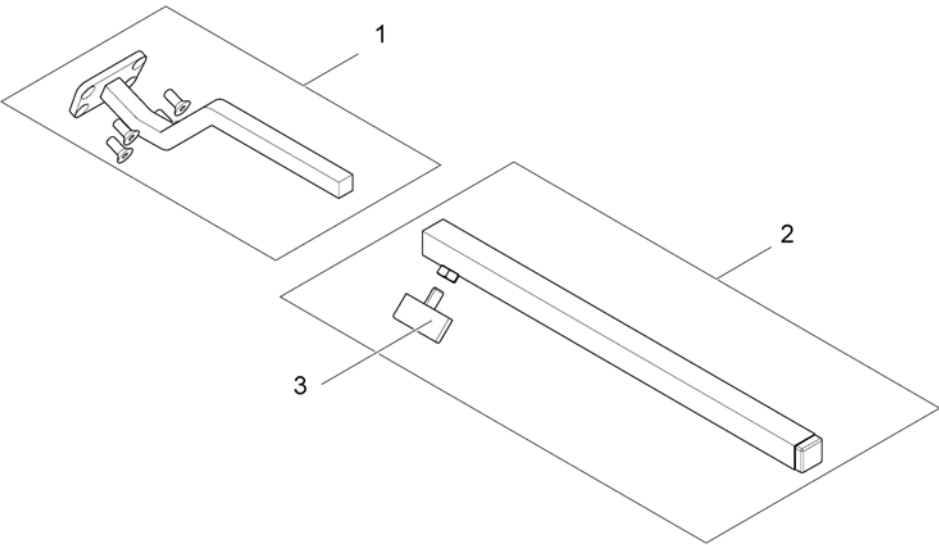

Pos.	Part number	Description	Valid from → until	Qty
1	SP1504822	Remote Support Standard		1
2	SPS183040	Remote Bracket, Standard		1
3	L31001	Wing ScREW M6X10		1

Remote Bracket Swing away & height adjust.


1560310.iso

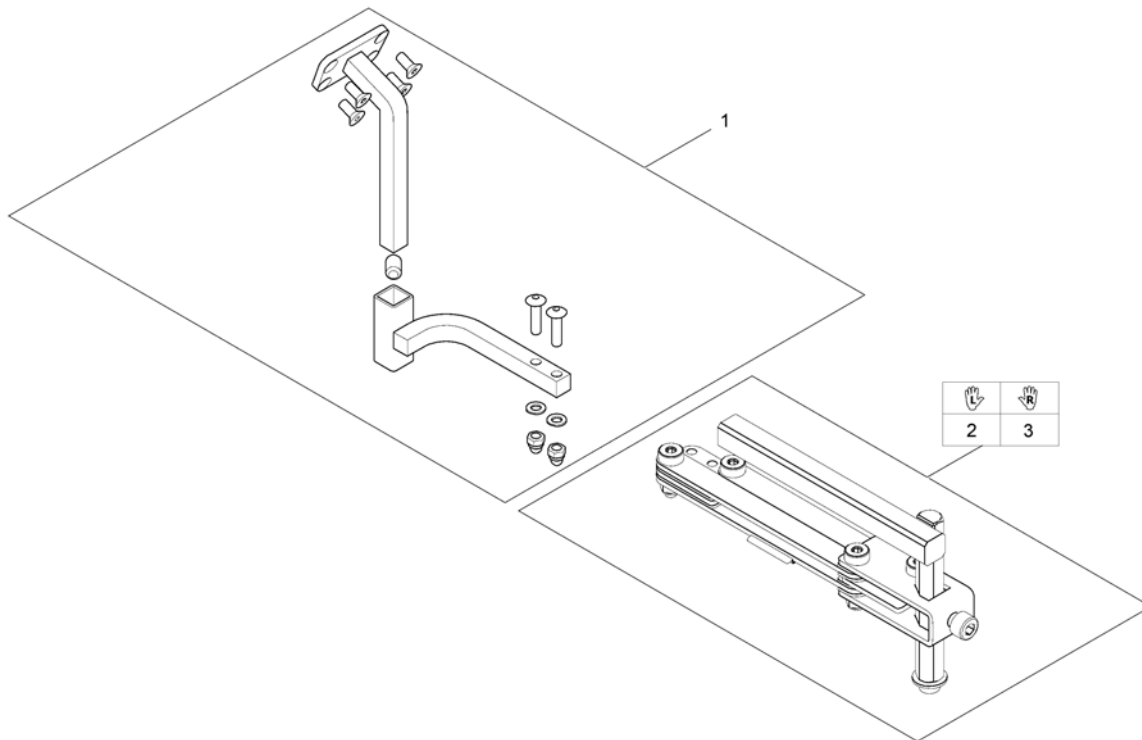

Pos.	Part number	Description	Valid from → until	Qty
1	SP1517352	Remote Support for Bracket height adjustable and swing away		1
2	SP1496249	Remote bracket, height adjustable & swing away, LH		1
3	SP1496248	Remote bracket, height adjustable & swing away, RH		1

Remote Bracket Attendant for Standard-/EasyAdapt-Seat


1560853.iso

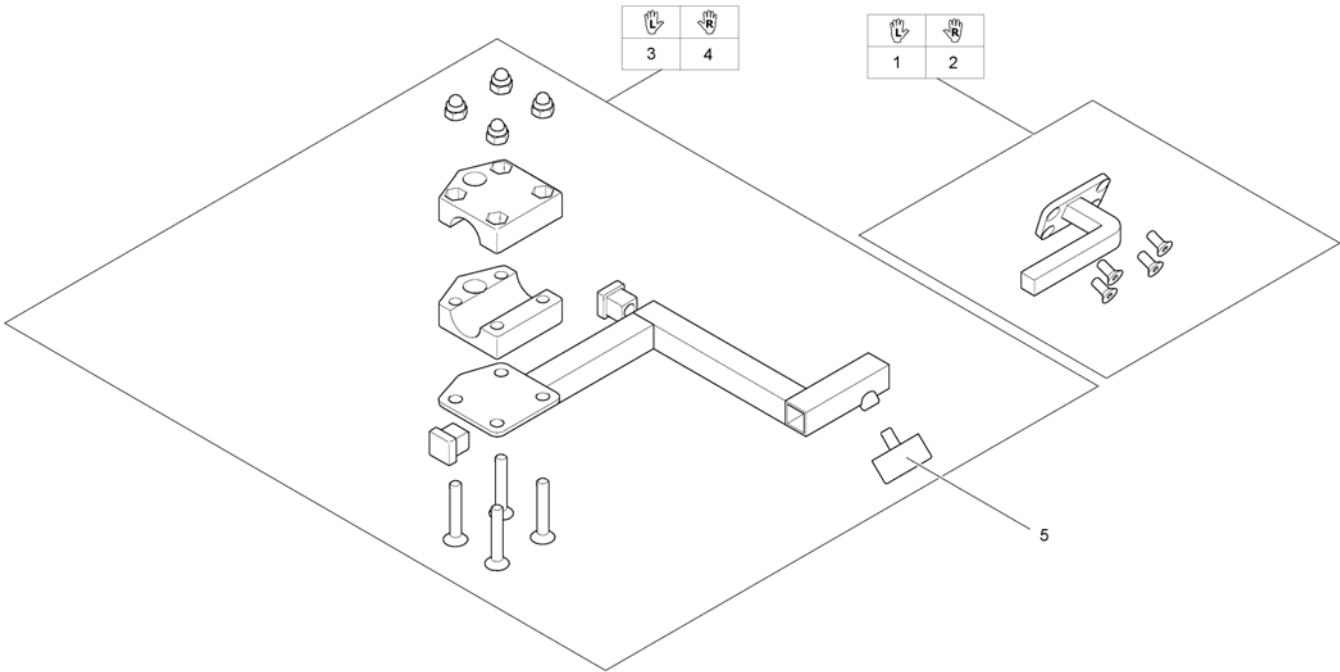



Pos.	Part number	Description	Valid from → until	Qty
1	SP1509435	Remote Support attendant control G90A, LH		1
2	SP1509439	Remote Support attendant control G90A, RH		1
3	SP1422937	Bracket for attendant control LH		1
4	SP1422936	Bracket for attendant control RH		1
5	L31001	Wing ScREW M6X10		1

Remote Bracket Attendant for Recaro-Seat

1563503.iso

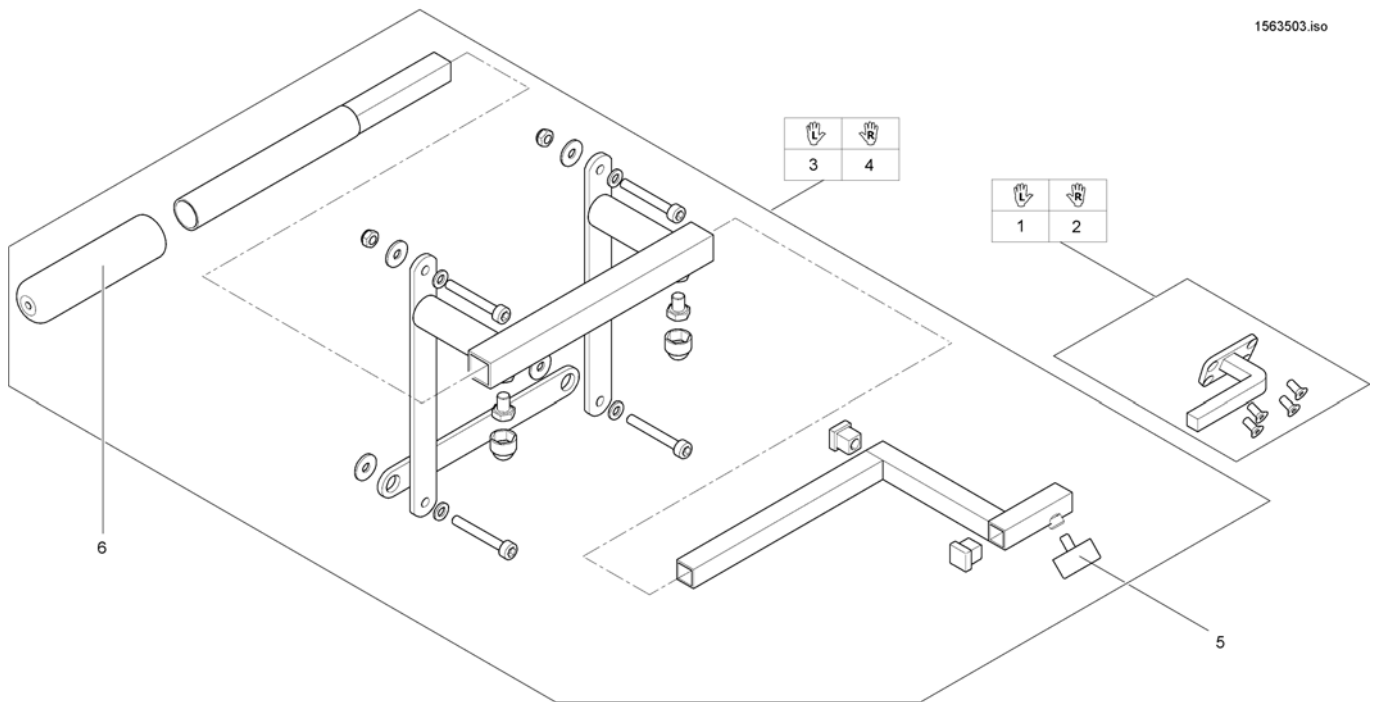


Pos.	Part number	Description	Valid from → until	Qty
1	SP1509435	Remote Support attendant control G90A, LH		1
2	SP1509439	Remote Support attendant control G90A, RH		1
3	SP1422814	Assembly kit for dual control cpl., Recaro, LH		1
4	SP1422813	Assembly kit for dual control cpl., Recaro, RH		1
5	L31001	Wing ScREW M6X10		1
6	1435421	Handle plastic black Ø22 "Invacare"		1

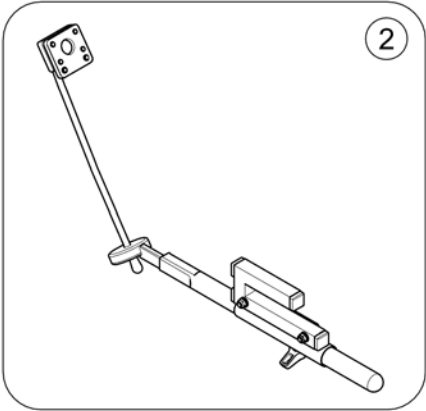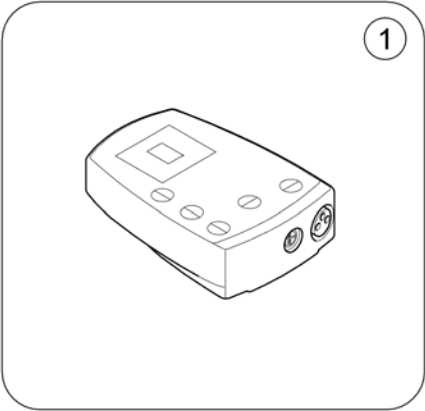

Pos.	Part number	Description	Valid from → until	Qty
1	SP1505187	Remote Control G91S [1615]		1
2	-	Remote Bracket Standard [1615+1207/1208]		1

Remote Bracket Standard

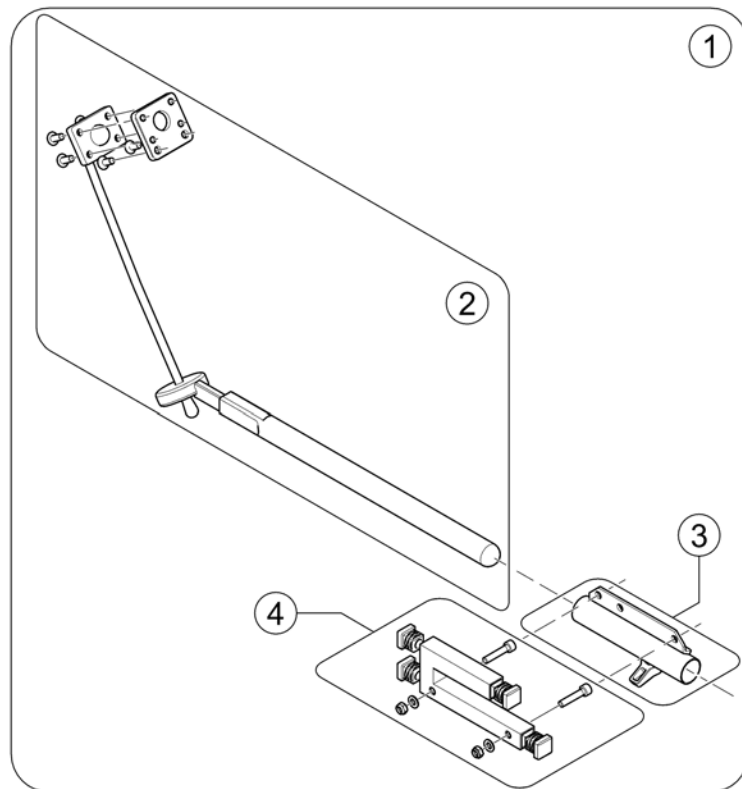


150054.iss

Pos.	Part number	Description	Valid from → until	Qty
1	SP1509690	Remote Bracket G91/G91S LH/RH, cpl.		1
2	SP1527504	Swan neck remote bracket with adapter plate		1
3	SP1603801	Mounting bracket, cpl. <i>Old bracket no longer available!</i>	2013-08-05 →	1
4	SP1603847	Mounting Adapter G91 <i>Old bracket no longer available!</i>	2013-08-05 →	1

Centre control REM 41

F383057.dwg


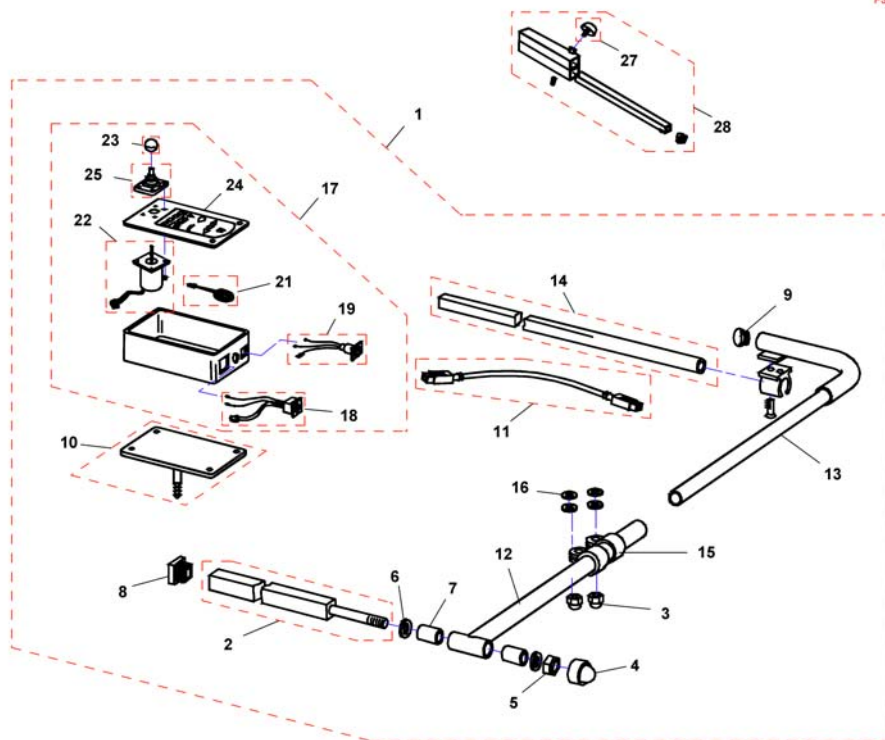


Pos.	Part number	Description	Valid from → until	Qty
-	Q99081	User Manual Table- & Middle Control REM41-S German		1
1	SP1530588	CENTRAL STEERING CPL.; SWING-AWAY RH, REM41-D		1
1	SP1530589	CENTRAL STEERING CPL.; SWING-AWAY LH, REM41-D		1
2	Z177024.00	SUPPORT TUBE; CHROME PLATED		1
3	F60208	Cap nut DIN1587-M08 zc		1
4	L01037	Sks Nut Cap M12		1
5	F20106	Hexagon nut DIN985-M12 zc		1
6	G00104	Washer DIN125-Ø13 zc		1
7	Z17032A	Bush Rd. 19 X 3.5 X 25		1
8	L01055	Plug 15x15mm		1
9	L01008	Tube End Plug SI 22		1
10	Z183216.P40	SUPPORT BRACKET; RAL 9005 BLACK		1
11	1559862	Bus cable ACS, 1700mm		1
12	Z183089.00	SWING ARM; CHROME PLATED		1
13	Z183086.00	SUPPORT ARM RH; CHROME PLATED		1
13	Z183092.00	SUPPORT ARM LH; CHROME PLATED		1
14	H281005A.00	MIRROR SUPPORT RAIL; CHROME PLATED		1
15	5006720	Klemmschelle F. Dm 22 mm		1
16	G00106	Washer DIN125-A-Ø8,4 zc		1
17	SP1527930	DX ACS Tray Remote DX-REM41		1
18	P41108	ACS-BUS CABLE SOCKET		1
19	P41106	CHARGER SOCKET		1
21	P30493	HORN		1
22	GSM55120	ACS-remote WITH 180MM CABLE		1
23	L36999	Joystick Knopf Standard Ball		1
24	-	No longer available.		0
25	Z183001A	Rubber gaiter for ACS remote		1
26	Z183289	CENTRAL STEERING SUPPORT TUBE CPL.		1
27	L31001	Wing Screw M6X10		1
28	Z183289	CENTRAL STEERING SUPPORT TUBE CPL.		1

Small Chin Control (old) with brackets


Pos.	Part number	Description	Valid from → until	Qty
-		Small Chincontrol		1
-		Framework for chincontrol		1

Small Chincontrol

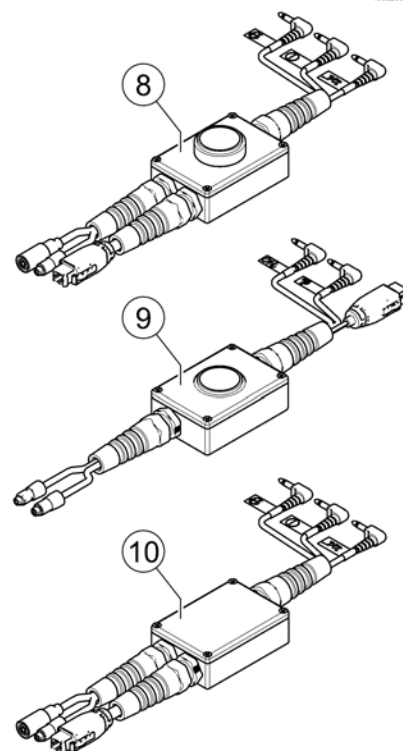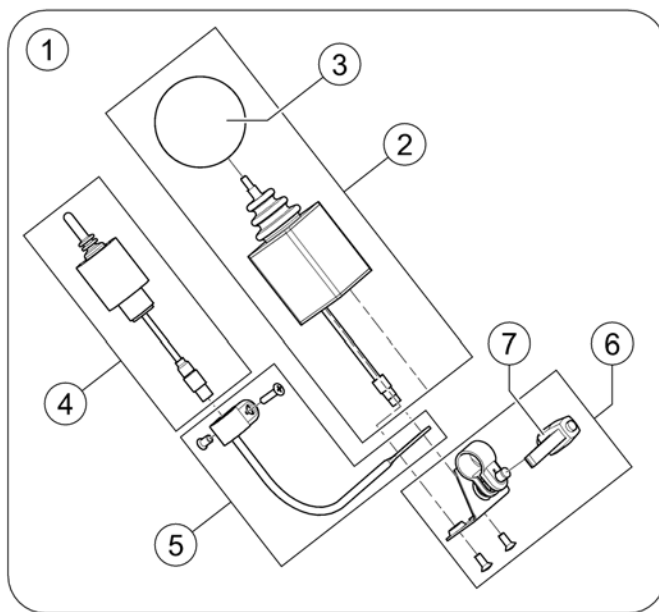

Pos.	Part number	Description	Valid from → until	Qty
1	1448705	Chin control REM24, cpl.		1
1	1421770	Chin control G80/G90, cpl.		1
1	SP1544255	Chin control G91, cpl.		1
2	Z183282	Joystick, DX-RJM (with Binder plug, 4-pole)		1
3	Z183265	Joystick Knob Sponge Ball		1
4	1442943	Switchjoystick for REM 24 SD, cpl.		1
4	Z183283	Switchjoystick G80/G90, cpl.		1
4	SP1544250	Switchjoystick for G91S, cpl.		1
5	SP1552294	Switchjoystick bracket		1
6	SP1552295	Joystick clamp		1
7	L32901	Clamping lever		1
8	SP1442944	Harness Chin Control for REM 24 SD		1
9	Z183275	Harness Chin Control for G80/G90		1
10	SP1541715	Harness Chin Control for G91S		1



Framework for chincontrol

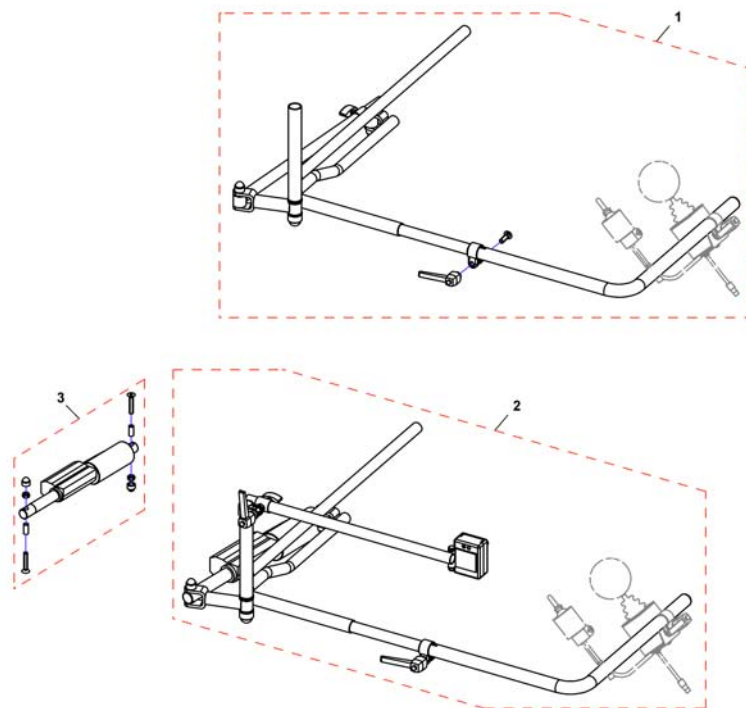



Pos.	Part number	Description	Valid from → until	Qty
-	Z167994	CHIN CONTROL SUPPORT . CPL. <i>Not shown / Nicht abgebildet / Non illustré</i>		1
1	SP1422943	Framework for chin control, manual, horizontal swing away, LH, cpl.		1
1	SP1422942	Framework for chin control, manual, horizontal swing away, RH, cpl.		1
2	SP1422945	Framework for chincontrol, powered, horizontal swing away, LH, cpl. <i>incl. Actuator</i>		1
2	SP1422944	Framework for chin control, powered, horizontal swing away, RH, cpl. <i>incl. Actuator</i>		1
-	SP1552335	Hardware, chin control, powered, horizontal swing away, LH <i>Not shown / Nicht abgebildet / Non illustré</i>		1
-	SP1552336	Hardware, chin control, powered, horizontal swing away, RH <i>Not shown / Nicht abgebildet / Non illustré</i>		1
-	SP1422946	Framework for chincontrol, powered, vertikal swing away <i>Not shown / Nicht abgebildet / Non illustré</i>		1
3	SP1552337	Actuator for horizontal swing away chin control, cpl.		1
3	SP1552418	Gestell für Kinnbedienung, elektrisch, vertikal abschwengbar <i>Not shown / Nicht abgebildet / Non illustré</i>		1

Chin Control (new) with mounting interface - Modulite Seat

1591328.iss

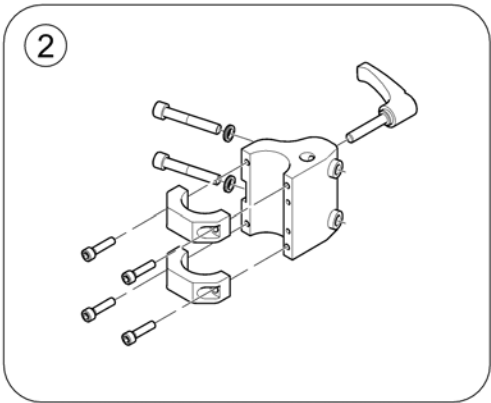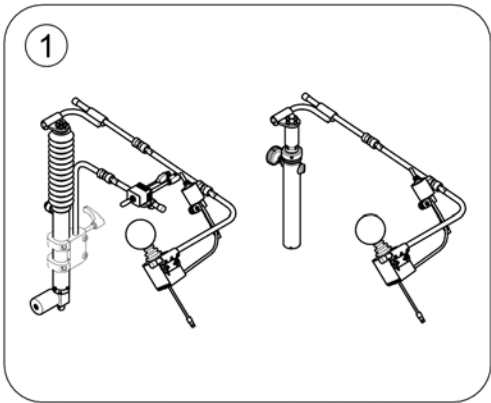


Pos.	Part number	Description	Valid from → until	Qty
1	-	Chin Control DX-RJM		1
2	-	Mounting Hardware - Chin Control		1

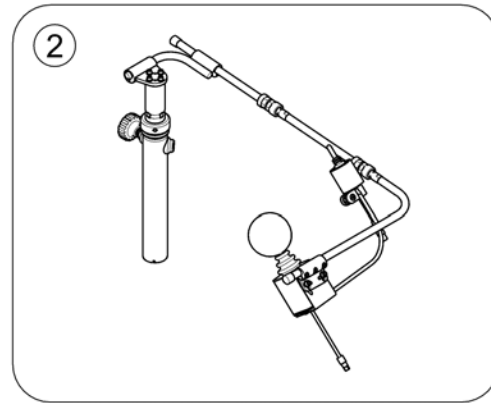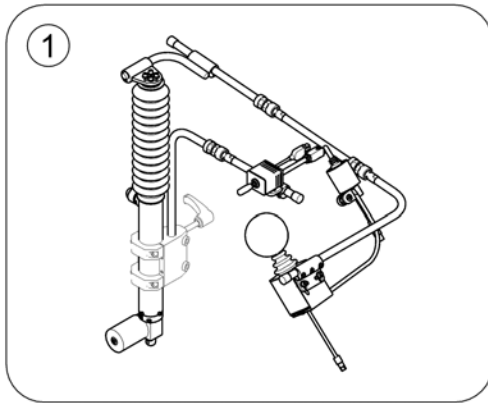


Pos.	Part number	Description	Valid from → until	Qty
1	-	Chin Control powered - Details		1
2	-	Chin Control manual - Details		1
-	SP1601389	Kit, Swing-away mounting arm, powered, LH, cpl. with stereo-switch (no joystick nor harness included) <i>Please order respective harness & joystick separately!</i>		1
-	SP1601390	Kit, Swing-away mounting arm, powered, RH, cpl. with stereo-switch (no joystick nor harness included) <i>Please order respective harness & joystick separately!</i>		1
-	SP1601391	Kit, Swing-away mounting arm, manual, LH, cpl. (no joystick nor harness included) <i>Please order respective harness & joystick separately!</i>		1
-	SP1601408	Kit, Swing-away mounting arm, manual, RH, cpl. (no joystick nor harness included) <i>Please order respective harness & joystick separately!</i>		1
-	SP1601528	Kit, Joysticks & Harness for REM 24 SD		1
-	SP1601529	Kit, Joysticks & Harness for G91S		1

Chin Control powered - Details

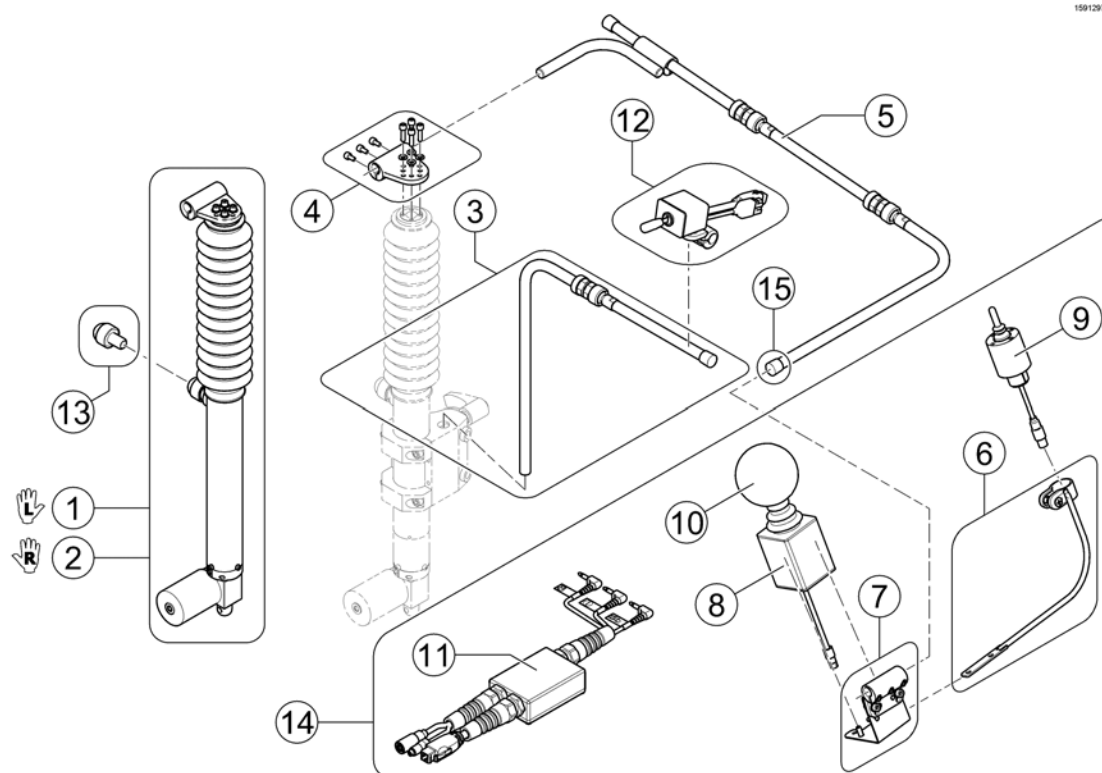

Pos.	Part number	Description	Valid from → until	Qty
-	SP1605661	Fixation Kit (cable ties, spiral coiled tube & end caps)	2016-04-01 →	1
1	SP1580094	Actuator cpl. LH		1
2	SP1591292	Actuator cpl. RH		1
3	SP1605659	Remote Rod Assy <i>New designed bracket (2016-04-01 =>) for Modulite Seat needed!</i>	2016-04-01 →	1
4	SP1589144	Rod cylinder connection chin, cpl.		1
5	SP1605660	Joystick Rod Assy	2016-04-01 →	1
6	SP1589146	Mounting arm for Switch Joystick		1
7	SP1589142	Mounting hardware for DX-RJM, cpl.		1
8	Z183282	Joystick, DX-RJM (with Binder plug, 4-pole)		1
9	1442943	Switchjoystick for REM 24 SD, cpl.		1
9	SP1544250	Switchjoystick for G91S, cpl.		1
10	Z183265	Joystick Knob Sponge Ball		1
11	SP1442944	Harness Chin Control for REM 24 SD		1
11	SP1541715	Harness Chin Control for G91S		1
12	SP1580701	Stereo-switch box, cpl. with mounting hardware		1
13	SP1589145	Adjustment Screw		1
14	SP1601528	Kit, Joysticks & Harness for REM 24 SD		1
14	SP1601529	Kit, Joysticks & Harness for G91S		1
15	SP1602721	End cap A12, black	2016-04-01 →	3


Chin Control manual - Details

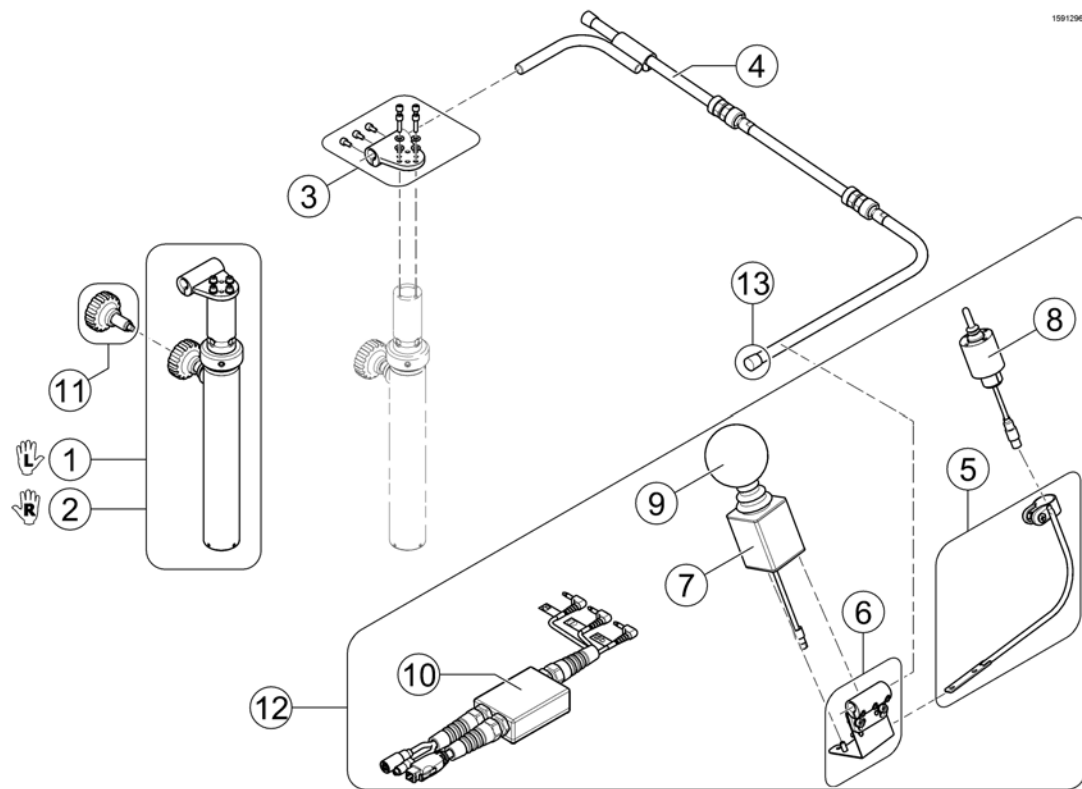

Pos.	Part number	Description	Valid from → until	Qty
-	SP1605661	Fixation Kit (cable ties, spiral coiled tube & end caps)	2016-04-01 →	1
1	SP1577456	ASM Chin Control man. LH, cpl.		1
2	SP1591293	ASM Chin Control man. RH, cpl.		1
3	SP1589144	Rod cylinder connection chin, cpl.		1
4	SP1605660	Joystick Rod Assy	2016-04-01 →	1
5	SP1589146	Mounting arm for Switch Joystick		1
6	SP1589142	Mounting hardware for DX-RJM, cpl.		1
7	Z183282	Joystick, DX-RJM (with Binder plug, 4-pole)		1
8	1442943	Switchjoystick for REM 24 SD, cpl.		1
8	SP1544250	Switchjoystick for G91S, cpl.		1
9	Z183265	Joystick Knob Sponge Ball		1
10	SP1442944	Harness Chin Control for REM 24 SD		1
10	SP1541715	Harness Chin Control for G91S		1
11	SP1580666	Adjustment Handle		1
12	SP1601528	Kit, Joysticks & Harness for REM 24 SD		1
12	SP1601529	Kit, Joysticks & Harness for G91S		1
13	SP1602721	End cap A12, black	2016-04-01 →	2

Mounting Hardware - Chin Control


1591309 iso

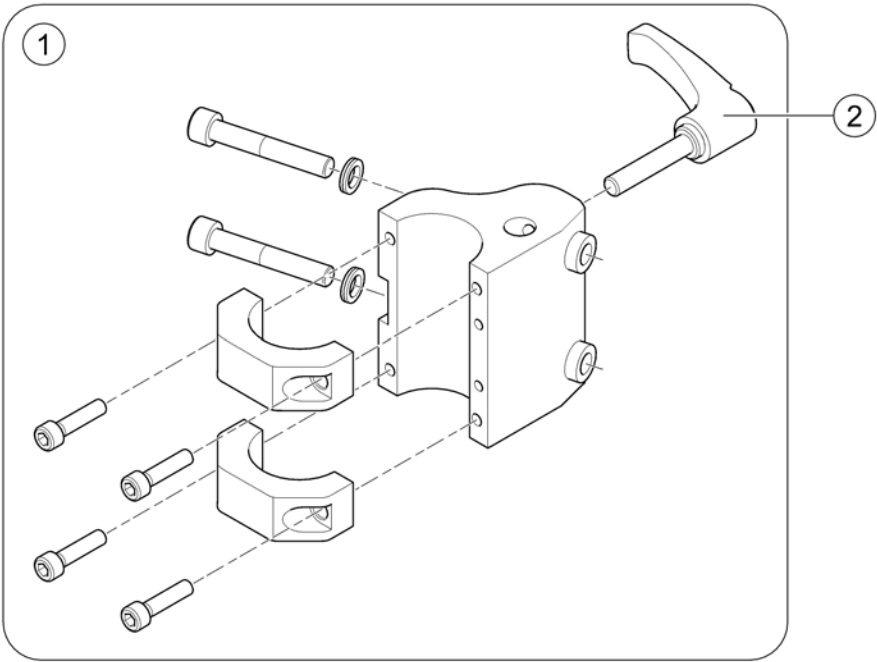

Pos.	Part number	Description	Valid from → until	Qty
1	SP1577173	Mounting Hardware, Chin Control - Modulite Seat <i>Design change 2016-04-01 =></i>		1
2	SP1604355	Clamp Lever M8x40		1

Dual Control with brackets


1564180-000

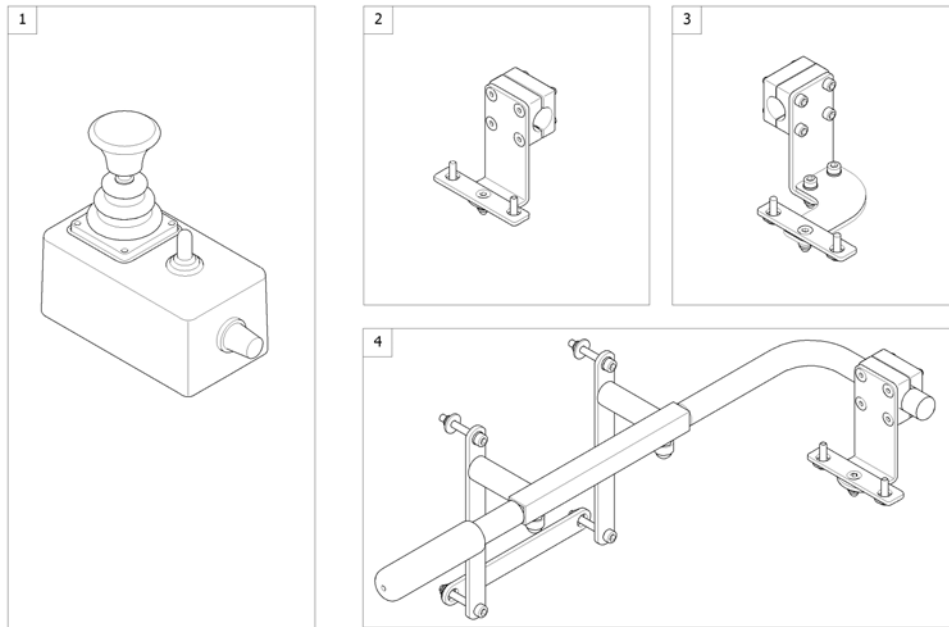

Pos.	Part number	Description	Valid from → until	Qty
1	6001425	ACS dual control [1291/1292]		1
2	-	Remote Bracket Dual Control for Standard-/EasyAdapt-Seat [1291/1292]		1
3	-	Remote Bracket Dual Control for Modulite Seat		1
4	-	Remote Bracket Dual Control for Recaro Seat [1291/1292]		1

Remote Bracket Dual Control for Standard-/EasyAdapt-Seat


1560885.iso

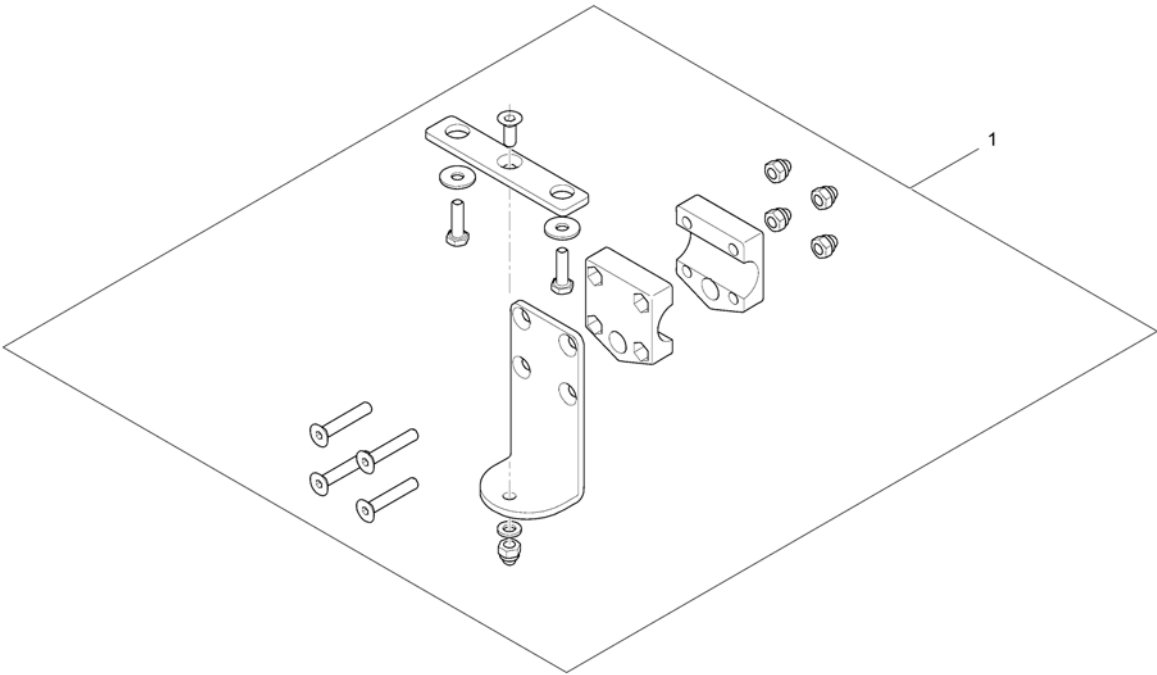

Pos.	Part number	Description	Valid from → until	Qty
1	SP1422818	Remote Bracket Dual Control		1

Remote Bracket Dual Control for Modulite Seat


1574291.iso

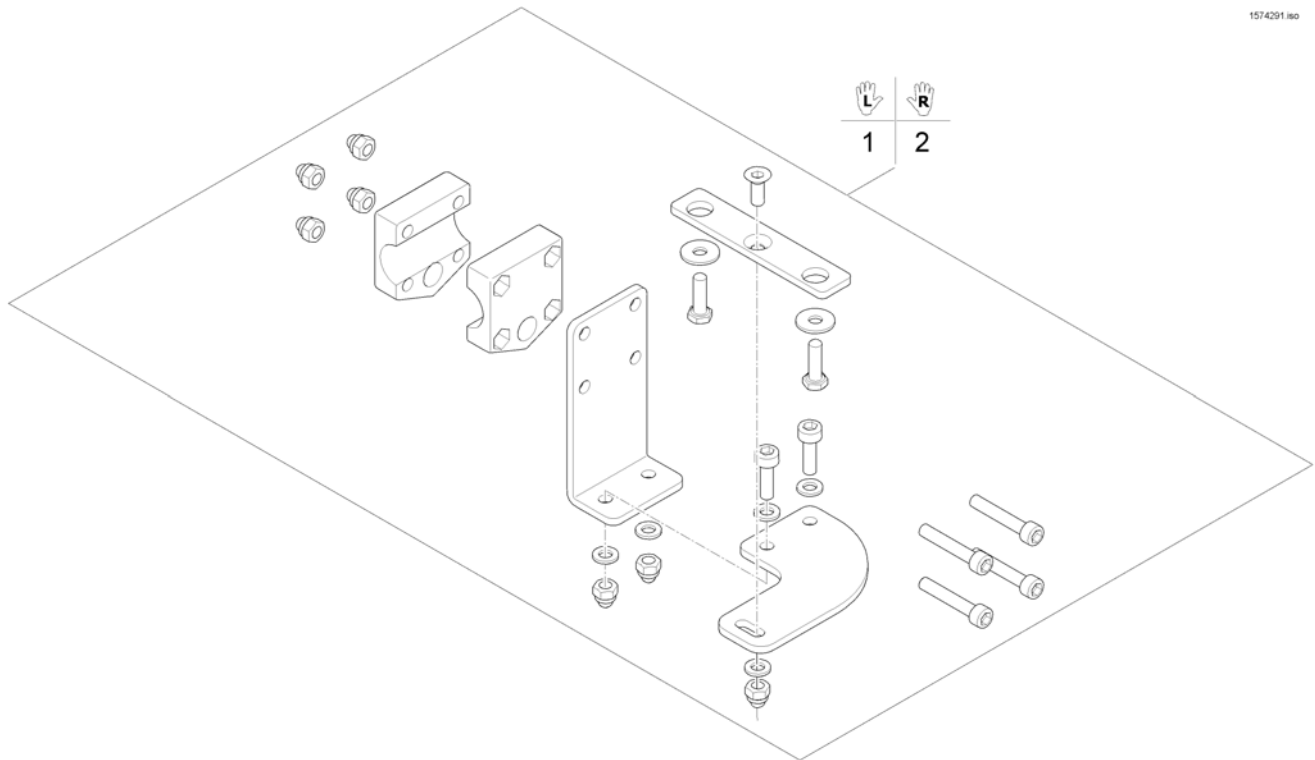

Pos.	Part number	Description	Valid from → until	Qty
1	SP1571799	Remote Bracket Dual Control LH		1
2	SP1571800	Remote Bracket Dual Control RH		1

Remote Bracket Dual Control for Recaro Seat


1563504.iso

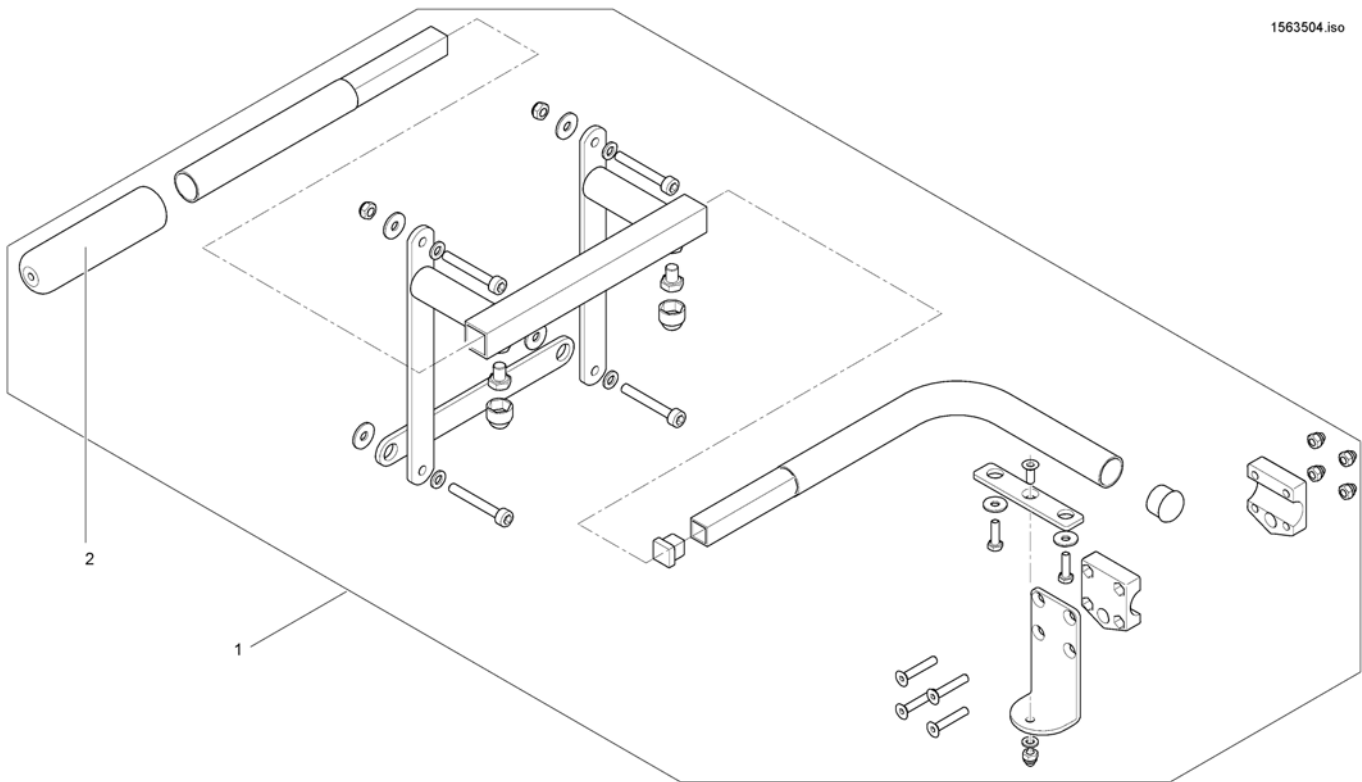

Pos.	Part number	Description	Valid from → until	Qty
1	SP1422819	Assembly kit for dual control Recaro		1
2	1435421	Handle plastic black Ø22 "Invacare"		1


Compact Dual Control with brackets

1508862 Rev

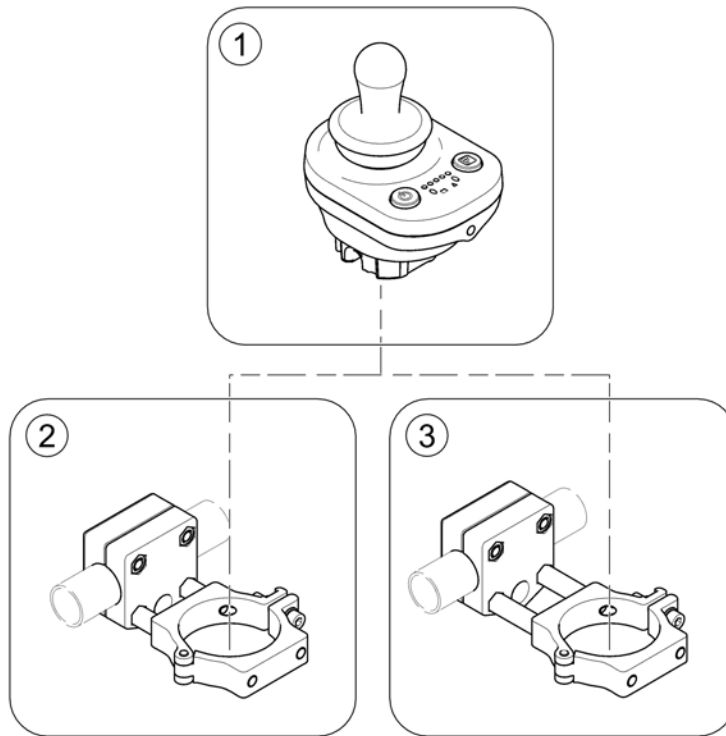

Pos.	Part number	Description	Valid from → until	Qty
1	-	Compact Dual Control		1
2	-	Bracket, Compact Dual Control - short		1
3	-	Bracket, Compact Dual Control - long		1


1586988.jpg

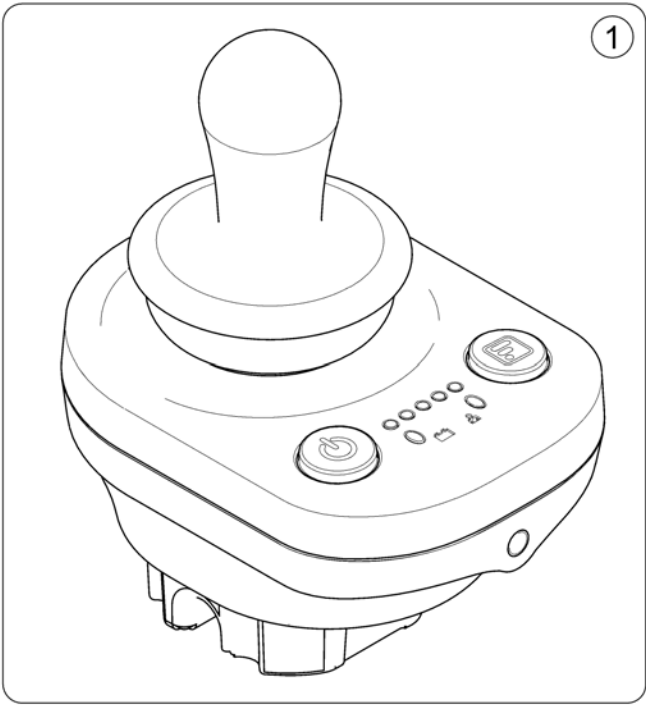

Pos.	Part number	Description	Valid from → until	Qty
1	SP1586988	Remote DX2-ACU		1

Bracket, Compact Dual Control - short


1588857.jpg

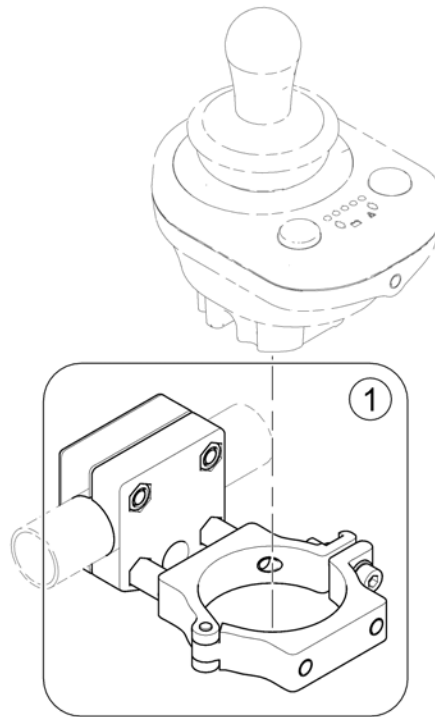

Pos.	Part number	Description	Valid from → until	Qty
1	SP1587904	Remote Attachment - short		1

Bracket, Compact Dual Control - long

158896.jpg

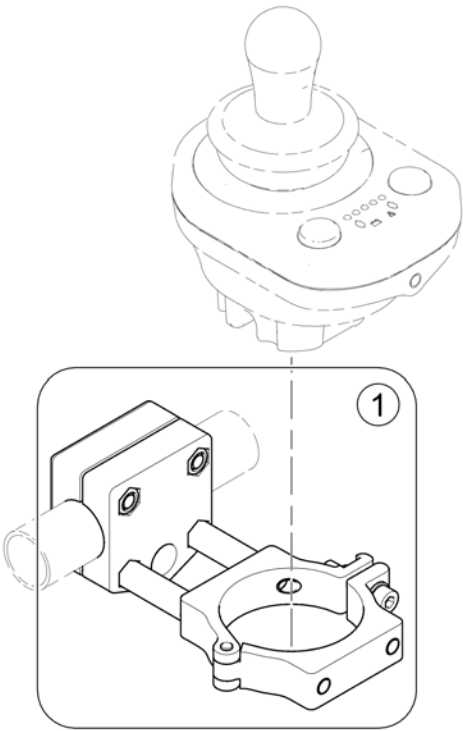


Pos.	Part number	Description	Valid from → until	Qty
1	SP1587119	Remote Attachment - long		1

Intuitive Dual Control (IDC) with bracket

Pos.	Part number	Description	Valid from → until	Qty
-	-	IDC with bracket		1
-	-	Height adjustable bracket IDC		1

IDC with bracket


1802413.iss

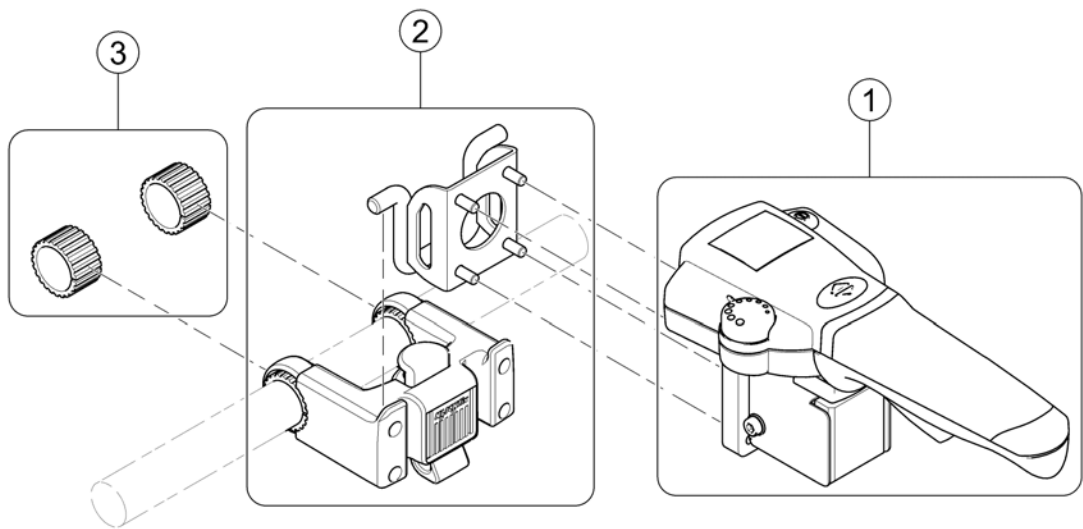

Pos.	Part number	Description	Valid from → until	Qty
-	-	The IDC incl. mounting hardware has been revised, the old hardware is no longer available.	2016-01-01 →	0
1	-	Intuitive Dual Control (IDC)		1
2	SP1603426	IDC Bracket		1
3	SP1601919	Rubber cuts		1

Intuitive Dual Control (IDC)


1603425-000

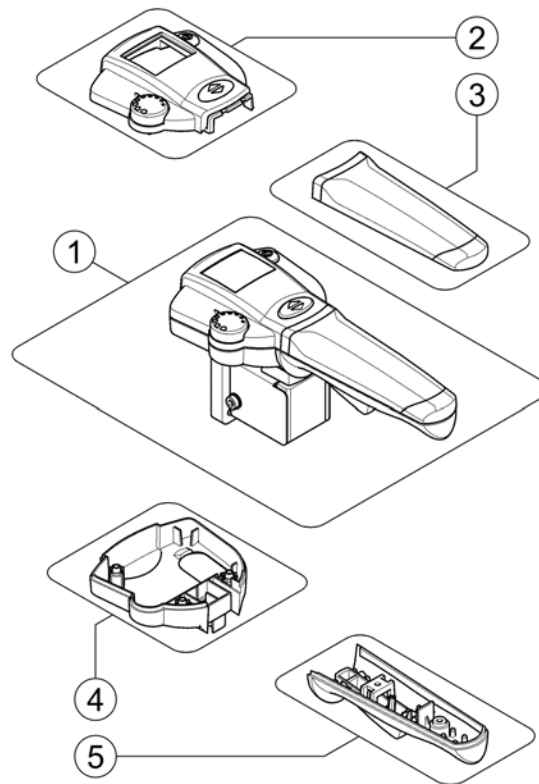

Pos.	Part number	Description	Valid from → until	Qty
-	-	If retrofitting, the user manual must be ordered separately!		1
-	-	User Manuals IDC (Intuitive Dual Control)		1
1	SP1603425	Intuitive Dual Control (revised), cpl.	2016-01-01 →	1
2	SP1579573	Upper Housing		1
3	SP1511060	Handhold upper part		1
4	SP1510470	Housing lower part		1
5	SP1511099	Handhold lower part		1

Height adjustable bracket IDC

1605464 iso

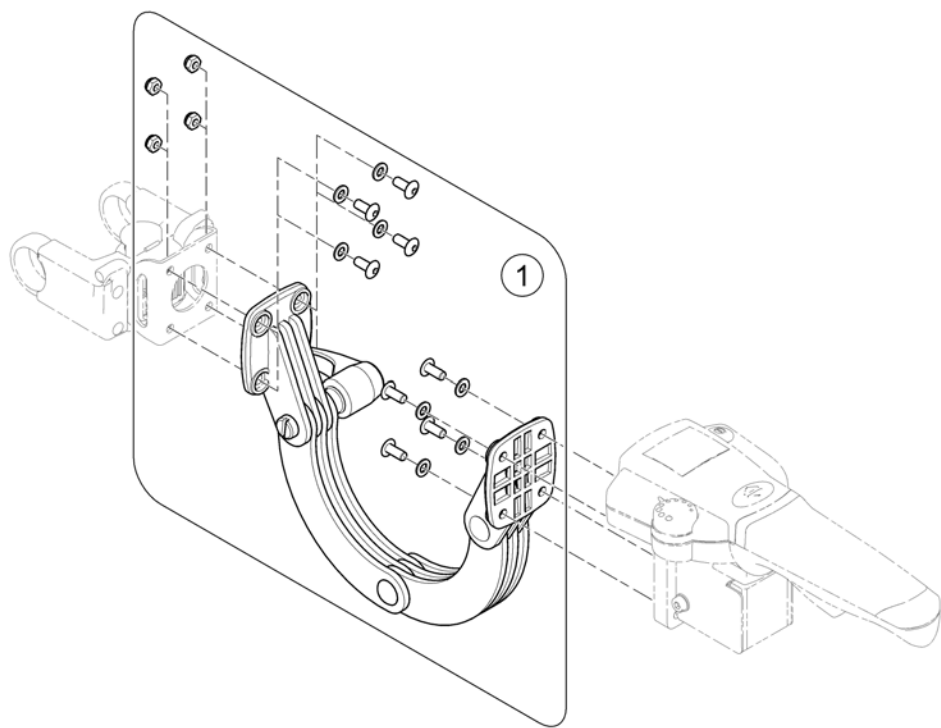


Pos.	Part number	Description	Valid from → until	Qty
1	SP1604195	Height adjustable bracket for IDC, cpl.		1

ACS Cables & Accessories

Pos.	Part number	Description	Valid from → until	Qty
-		Bus-Cables ACS		1
-		T-Adapter ACS		1
-		T-Adapter - Modulite Seat		1
-		Programming Kits		1
-		Magnetkey for Remotes		1

Bus-Cables ACS

ACSBUSKABEL.iso

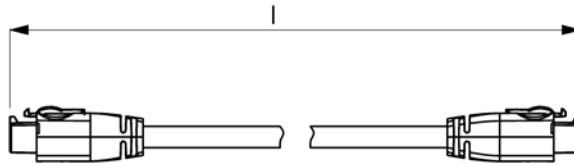


Position	length l [mm]
1	300
2	500
3	700
4	1100
5	1700
6	2000
7	2500
8	2700
9	1060

Pos.	Part number	Description	Valid from → until	Qty
1	GSM63003	Bus cable ACS, 300mm		1
2	SPGSM63005	Bus cable ACS, 500mm <i>Replace GSM63004</i>		1
3	GSM63007	Bus cable ACS 700mm		1
4	GSM63011	Bus cable ACS, 1100mm		1
5	1559862	Bus cable ACS, 1700mm <i>Replace GSM63016</i>		1
6	GSM63020	Bus cable ACS, 2000mm <i>Replace 1419386</i>		1
7	1440424	Bus cable ACS, 2500mm		1
8	SP1559873	Bus cable ACS, 2700mm <i>Replace GSM63026</i>		1
9	SP1550402	Extension cable ACS, 1060mm		1

T-Adapter ACS


1541385X.dwg

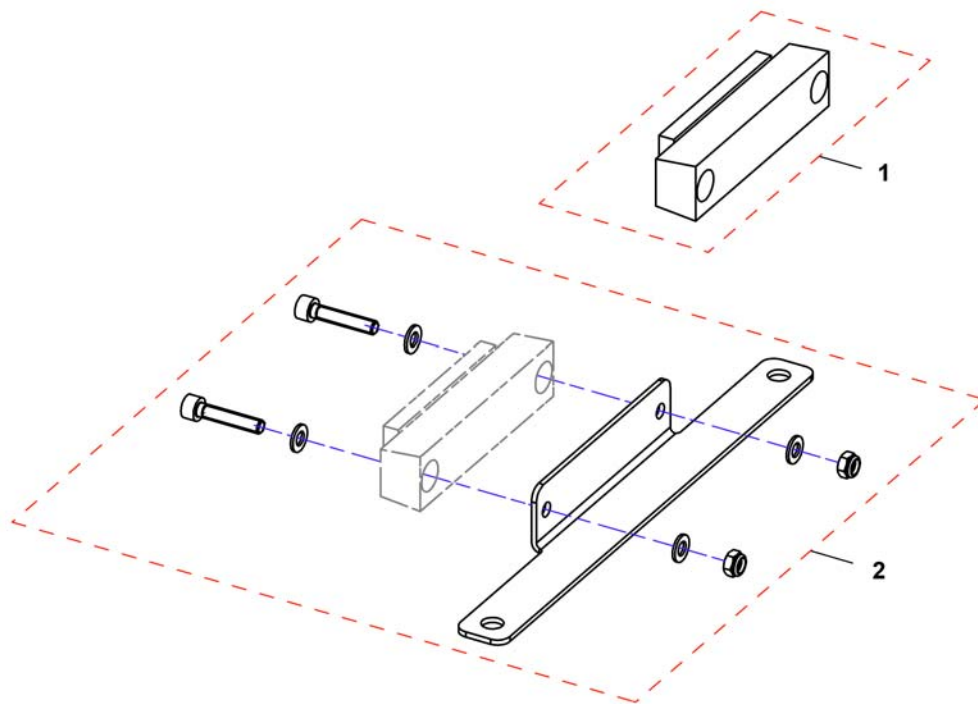


Pos.	Part number	Description	Valid from → until	Qty
1	P10072	BUS4 Way Socket		1
2	SP1541744	Holder T-Adapter		1

T-Adapter - Modulite Seat


1583360 iso

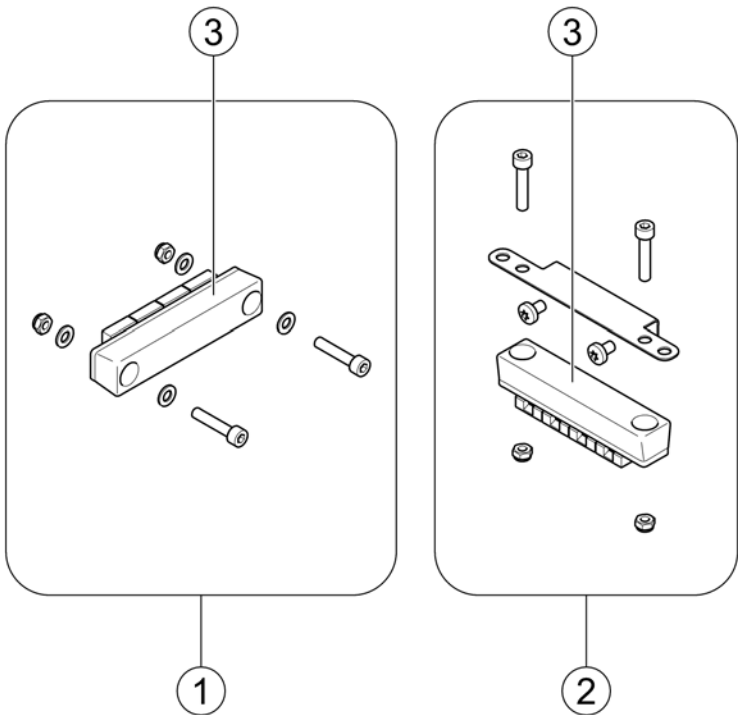

Pos.	Part number	Description	Valid from → until	Qty
1	SP1571826	T-Adapter - Modulite Seat		1
2	SP1569958	T-Adapter - Modulite Seat with MBL-Modul		1
3	P10072	BUS4 Way Socket		1

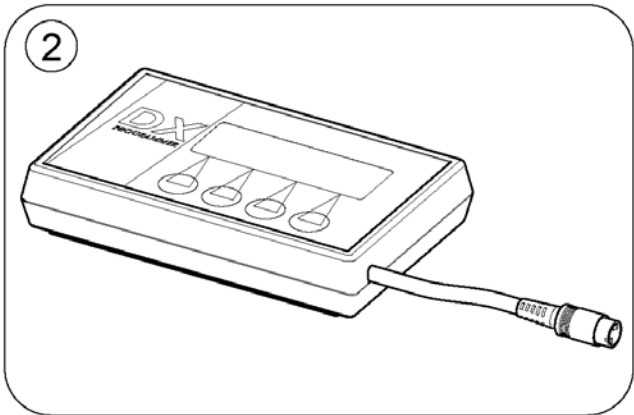

Pos.	Part number	Description	Valid from → until	Qty
1	1449280	USW wizard dongle kit, EDL-USW, cpl.		1
2	6001450	Hand Programmer, Instrucions In French, German, Spanish		1
	SP1441551	Programming Adapter Shark DK-ADAPT (necessary to program Shark)		1

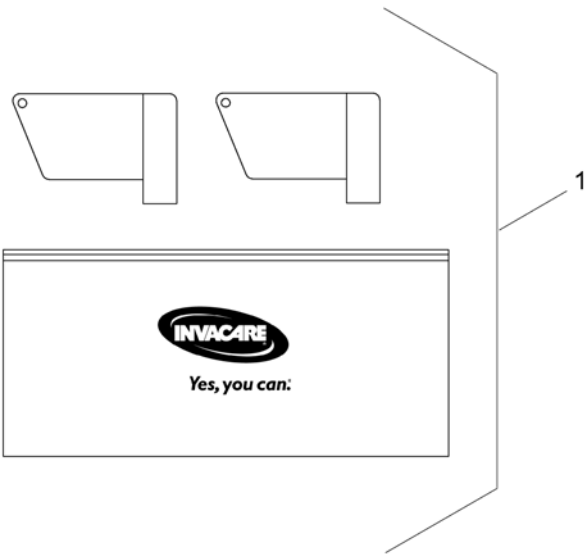


Pos.	Part number	Description	Valid from → until	Qty
1	SP1419695	Magnet key with bag		1

Pos.	Part number	Description	Valid from → until	Qty
-		iPortal Modules and Accessories		1
-		Remote, Joystick, Control		1
-		Joystick Knob		1
-		Buddy Button		1
-		Piko Button		1
-		Interface, Adapter, Other		1

iPortal Modules and Accessories


1603435 iso

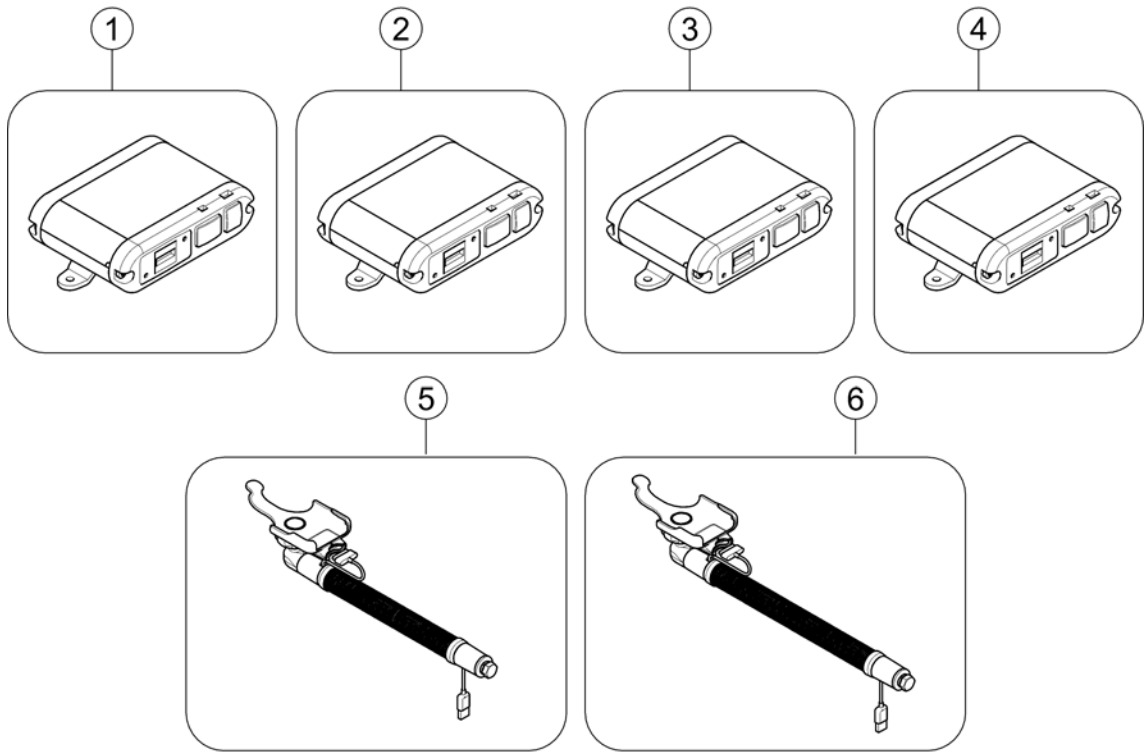

Pos.	Part number	Description	Valid from → until	Qty
1	SP1553373	iPortal Bluetooth Module Accessibility [1629]		1
2	SP1570392	iPortal Bluetooth Module Mouse Mover [1624]		1
3	SP1570393	iPortal Bluetooth Module Mouse Mover & Accessibility [1662]		1
4	SP1545530	iPortal Module without Accessibility [1634]		1
5	SP1545531	Gooseneck short for iPod touch & iPhone [1635]		1
6	SP1545532	Gooseneck long for iPod touch & iPhone [1636]		1

Remote, Joystick, Control


1603436.iss

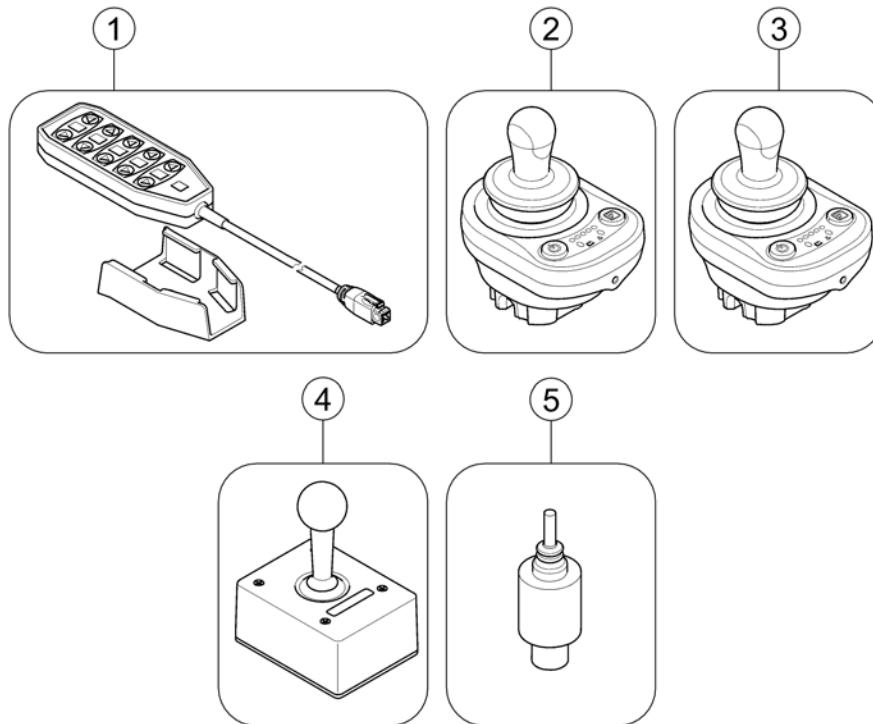

Pos.	Part number	Description	Valid from → until	Qty
1	P10079	Pendant control ARC 5 [1641/1642]		1
2	SP1586989	Remote Joystick [1621]		1
3	SP1586990	Remote Low-force Joystick [1722]		1
4	P10171	Heavy Duty Switch Joystick [1620]		1
5	P10175	Mini-Joystick with Push function [1622]		1
-	1442679	Head-Remote [1613]		1
-	SP1603442	Cable for Head-Remote ultrasonic proximity sensor		1

Joystick Knob


1603437.indd

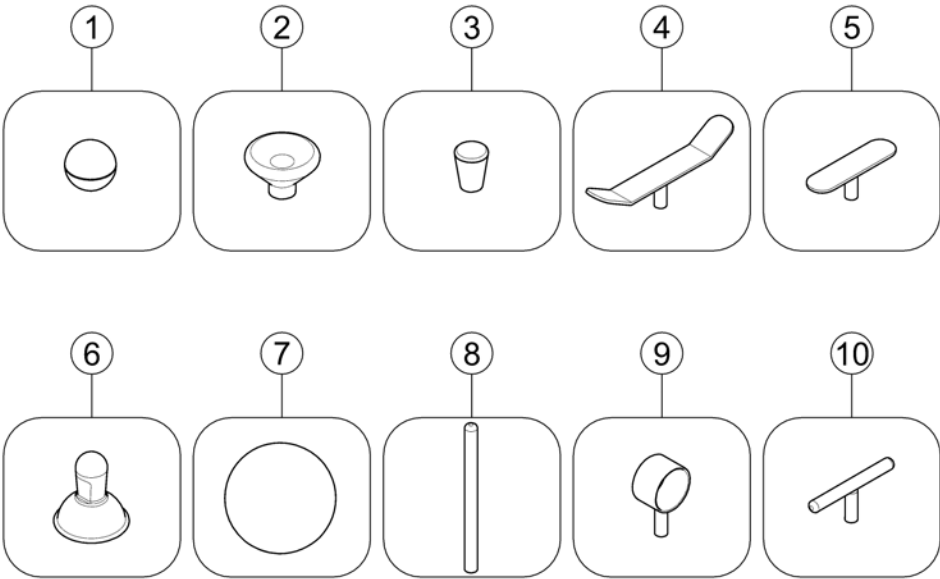

Pos.	Part number	Description	Valid from → until	Qty
1	L36999	Joystick Knopf Standard Ball [1650]		1
2	P10397	Joystick Knob Chin Cup [1651]		1
3	P39302	Joystick Knob Carrot [1652]		1
4	P39303	Joystick Knopf Flat T-Plate [1653]		1
5	P39305	Joystick Knob Flat T-Plate [1654]		1
6	P10306	Joystick Knopf DL [1655]		1
7	Z183265	Joystick Knob Sponge Ball [1656]		1
8	P39306	Joystick Knob Straight Bar [1657]		1
9	P39307	Joystick Knob Finger Tube [1658]		1
10	P39304	Joystick Knob Rount T-Bar [1659]		1

Buddy Button


1603436.000

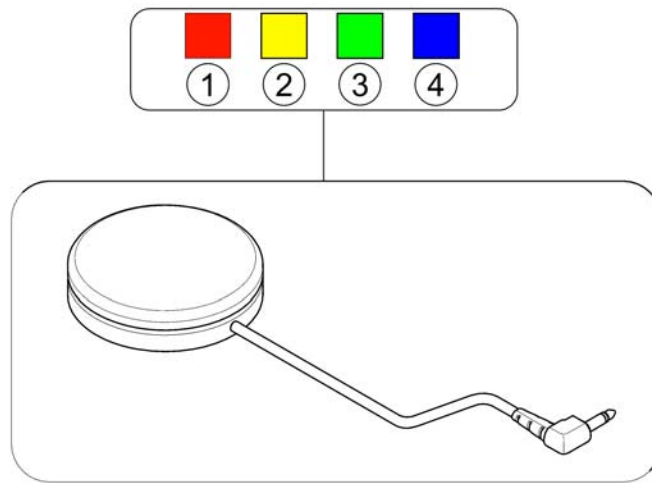


Pos.	Part number	Description	Valid from → until	Qty
1	5813964	Buddy Button red [1685]		1
2	P30093	Buddy Button yellow [1686]		1
3	P30092	Buddy Button green [1687]		1
4	P30089	Buddy Button blue [1690]		1

Piko Button

1503438.jpg

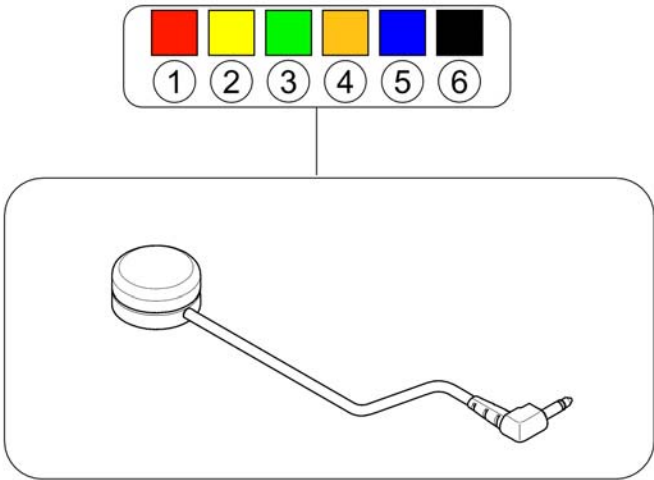

Pos.	Part number	Description	Valid from → until	Qty
1	SP1535717	Piko Button red [1711]		1
2	SP1535718	Piko Button yellow [1712]		1
3	SP1535723	Piko Button Green [1713]		1
4	SP1535724	Piko Button orange [1714]		1
5	SP1535725	Piko Button blue [1715]		1
6	SP1535726	Piko Button black [1716]		1

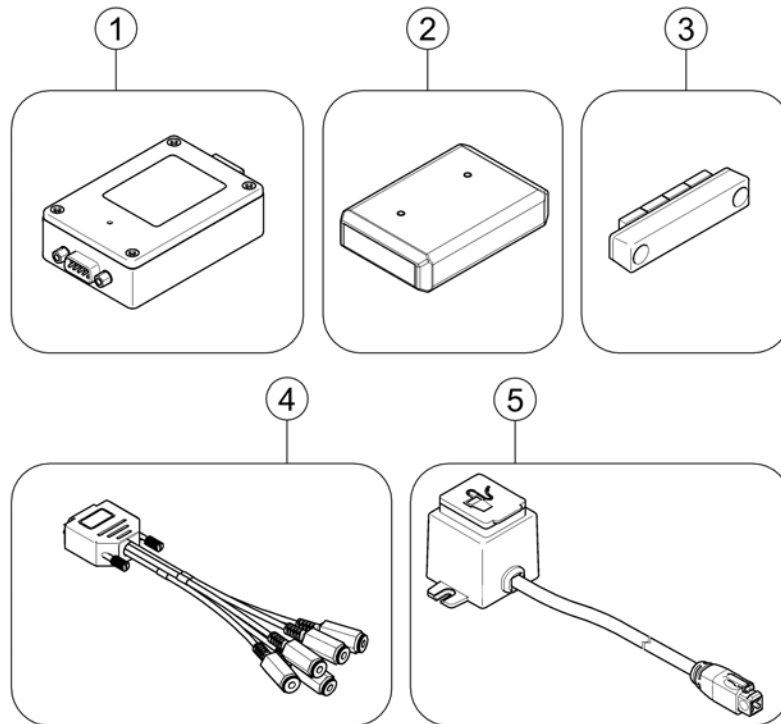

Pos.	Part number	Description	Valid from → until	Qty
1	SP1544036	4 Switch Interface [1630]		1
2	P10163	Sip & Puff Module(DX-SNP) [1633]		1
-	Z183267	Bracket for Sipp and Puff [1668]		1
3	P10072	BUS4 Way Socket [1680]		1
4	P30080	5-1 Switch Adaptor [1681]		1
5	P40107	Charger Socket Modul [1661]		1

Batteries

Batterys.iso


Product Name	Capacity (Ah)															
	1,2	12	30	31	32	36	38	40	40	50	50	50,9	60	70/73,6	75	97,6
Bora / SpectraXTR								8		10						
Dragon										10			13			
Dragon Vertic													13			
FDX													13			
G50													13	14		16
Kite													13	14		
Mirage				4				8								
Mistral				4								12				
P9000XDT					5											
Pronto M41					5											
Pronto M61					5											
Spectra Blitz								8				12				
Spectra Comfort							7									
Spectra Plus							7									
Storm3										10			13	14		
Storm3 True Track	1													14		
Storm3 Vertic														14		
Storm4 Max														14		
Stream										10			13			
TDX-SP => 2015/08/31														14		
TDX-SP NB => 2015/08/31												12				
Tiger										10			13			
Typhoon II	1												13			
Zephyr	1												13			
Comet														14	15	
Leo						6			9							
Lynx		2														
Orion											11					
Length (mm)	97	152	197	196	211	196	210	197	198	198	198	238	259	260	260	329
Width (mm)	49	99	132	132	132	130	175	168	165	166	165	140	169	171	168	171
Height (mm)	51	102	180	183	183	179	175	175	171	176	170	235	178	210	223	238
Weight (kg)	0,6	3,9	11,1	10,5	11	12	14,6	14,4	14,2	14	16	17,1	19,3	24,3	26	32,5



Pos.	Part number	Description	Valid from → until	Qty
-	-	Not listed products have their own battery page!		1
!	-	Important advice! Replace batteries in pairs and same type only!		1
1	1440386	Battery, 12V/1,2Ah C20, GEL		1
2	SP1506493	Battery, 12V/12Ah, AGM		1
4	P50029	Battery, 12V/31Ah C20, GEL		1
5	SP1521459	Battery, 12V/34Ah C20, AGM		1
6	1505498	Battery, 12V/36Ah C20, AGM		1
7	P50028	Battery, 12V/38Ah C20, GEL (UK & DK only)		1
8	PG50988	Battery, 12V/40Ah C20, GEL		1
9	SP1550692	Battery, 12V/40Ah C20, AGM		1
10	SP1522107	Battery, 12V/50Ah C20, AGM		1
11	SP1521695	Battery, 12V/50Ah C20 AGM		1
12	PG50989	Battery, 12V/50,9Ah C20, GEL		1
13	1432944	Battery, 12V/60Ah C20, GEL		1
14	P50016	Battery, 12V/70-73,6 Ah C20, Gel		1
15	SP1522026	Battery, 12V/75Ah C20, AGM		1
16	1478675	Battery, 12V/97,6Ah C20, GEL		1

Chargers


1589799.iso

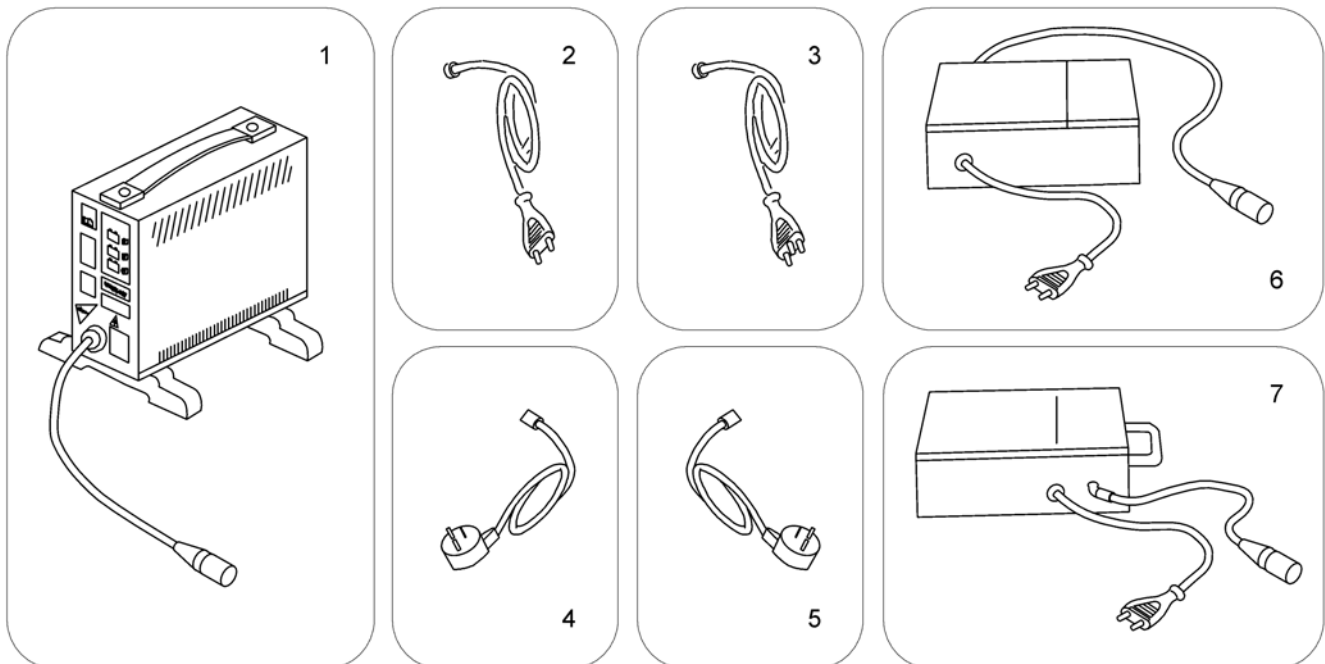

Pos.	Part number	Description	Valid from → until	Qty
1	SPP51981	Charger 8A without Power Cord [1343]		1
2	P21079	Power Cord Continental (Schuko GCB-0401)		1
3	P21080	Power Cord UK (GCB-0403)		1
4	1443841	Power Cord Swiss (black 6A/250V)		1
5	SP1553849	Power Cord Israel (IEC 320 C13 to SI-32)		1
6	SP1545511	Charger 10A incl. Power Cord Continental [1389]		1
7	5813987	Charger 8A (Protectionclass IP54) incl. Power Cord Continental (Sweden only) [1388]		1

Main harnesses


1167681X.dwg

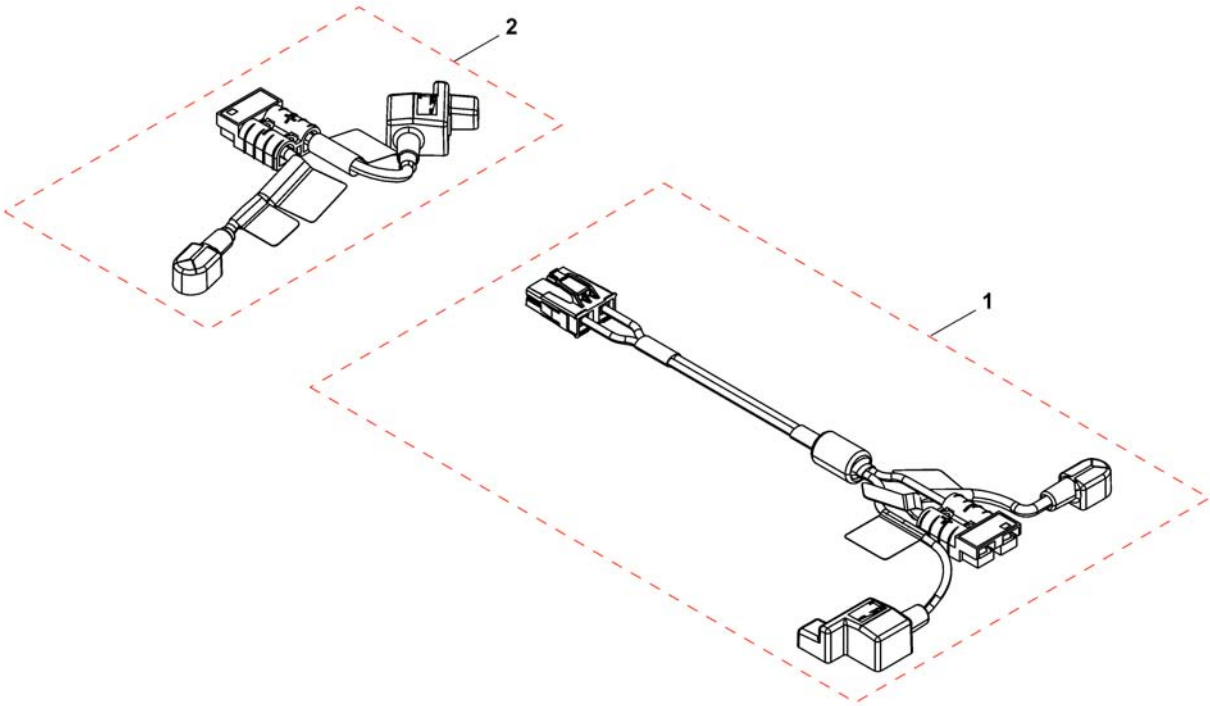

Pos.	Part number	Description	Valid from → until	Qty
1	SP1542998	Rear harness		1
2	SP1542999	Front harness		1

Light System (German homologation)


1537893X.dwg

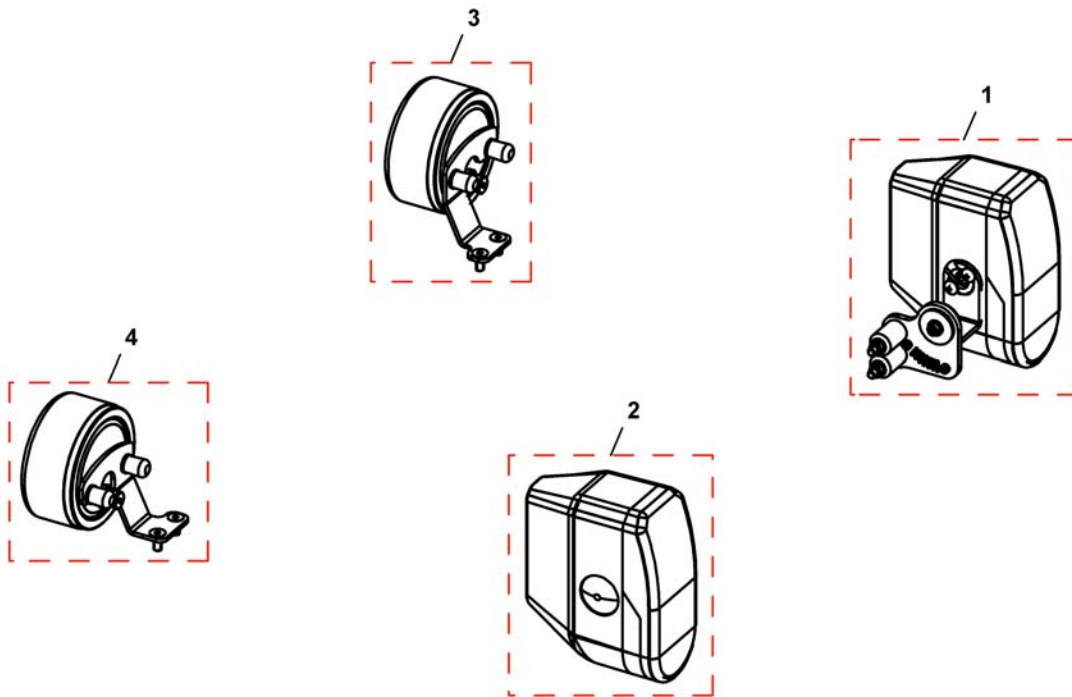

Pos.	Part number	Description	Valid from → until	Qty
!!!	-	Rear Light Protector Kit		0
1	SP1528319	Frontlight, complete with bracket, LH (Germany)		1
2	SP1528320	Frontlight, complete with bracket, RH (Germany)		1
3	SP1537891	Rearlight, cpl., RH		1
4	SP1537892	Rearlight, cpl., LH		1


Rear Light Protector Kit

1585420 (90)

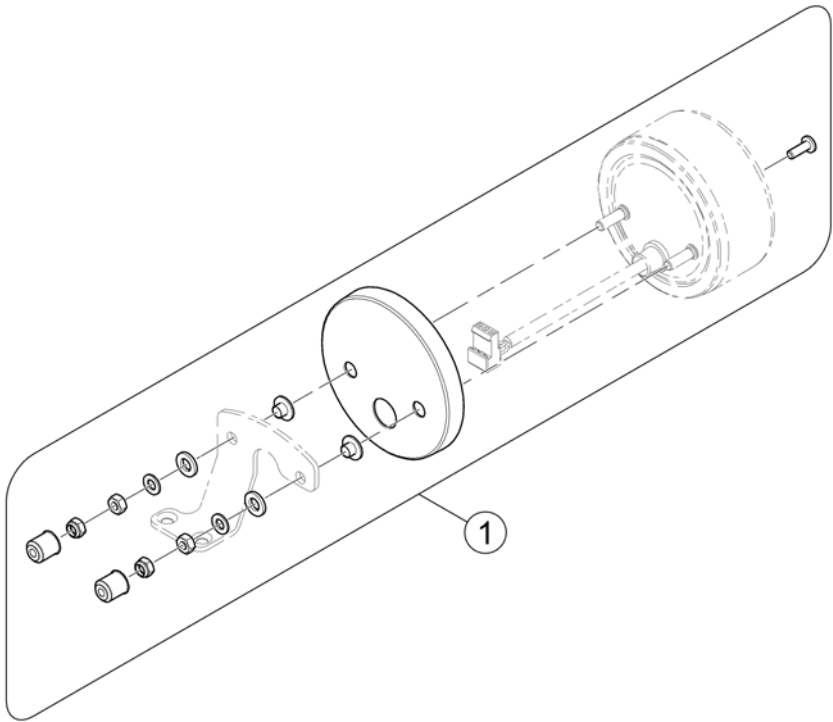

Pos.	Part number	Description	Valid from → until	Qty
1	SP1581827	Rear Light Protector Exchange Kit		1


Frontlight for Modulite (German homologation)

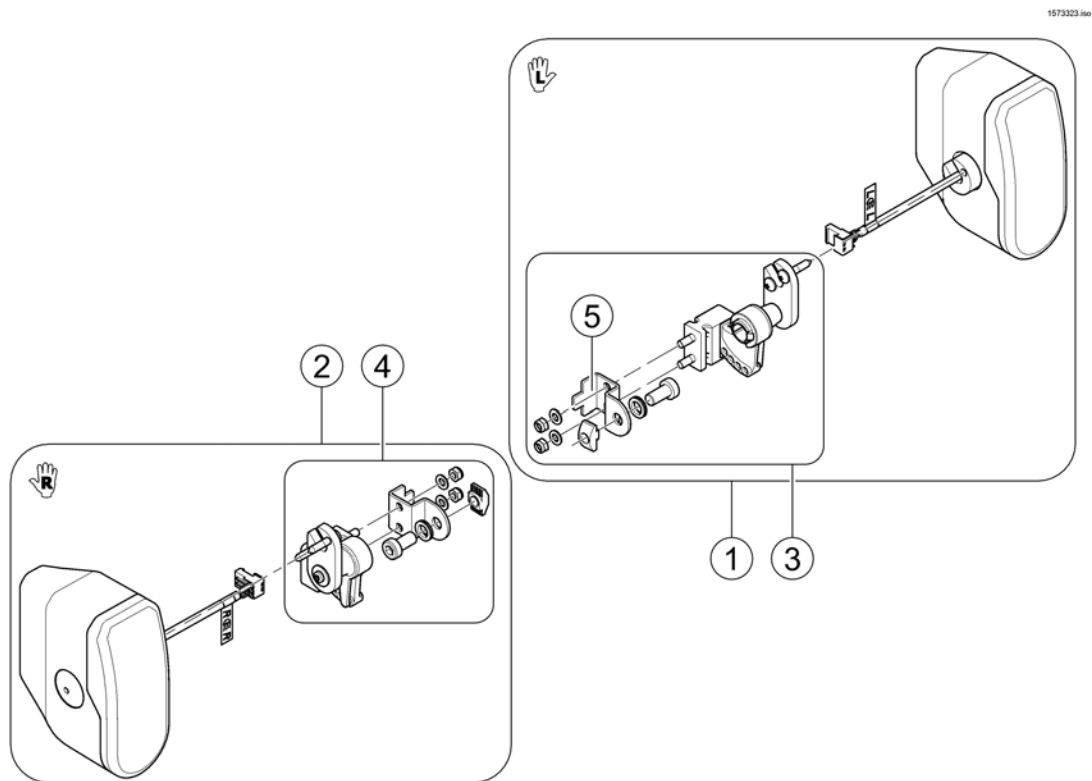

Pos.	Part number	Description	Valid from → until	Qty
-		Bracket design changed on 2015-01-10. For Retrofitting please order SP1556591.		1
1	SP1573237	Frontlight, complete with bracket, LH (Germany)		1
2	SP1573252	Frontlight, complete with bracket, RH (Germany)		1
3	SP1573253	Frontlight bracket, LH, cpl.		1
4	SP1573320	Frontlight bracket, RH, cpl.		1
5	SP1556591	Lamp bracket <i>New design!</i>	2015-01-10 →	2
-	SP1419271	Light Bulb 24V/21W, yellow (Turn Signal front/rear)		1
-	SPP31108	Light Bulb 24V/10W (BA15s), white (Front)		1

LED-Frontlight


1536030X.iso

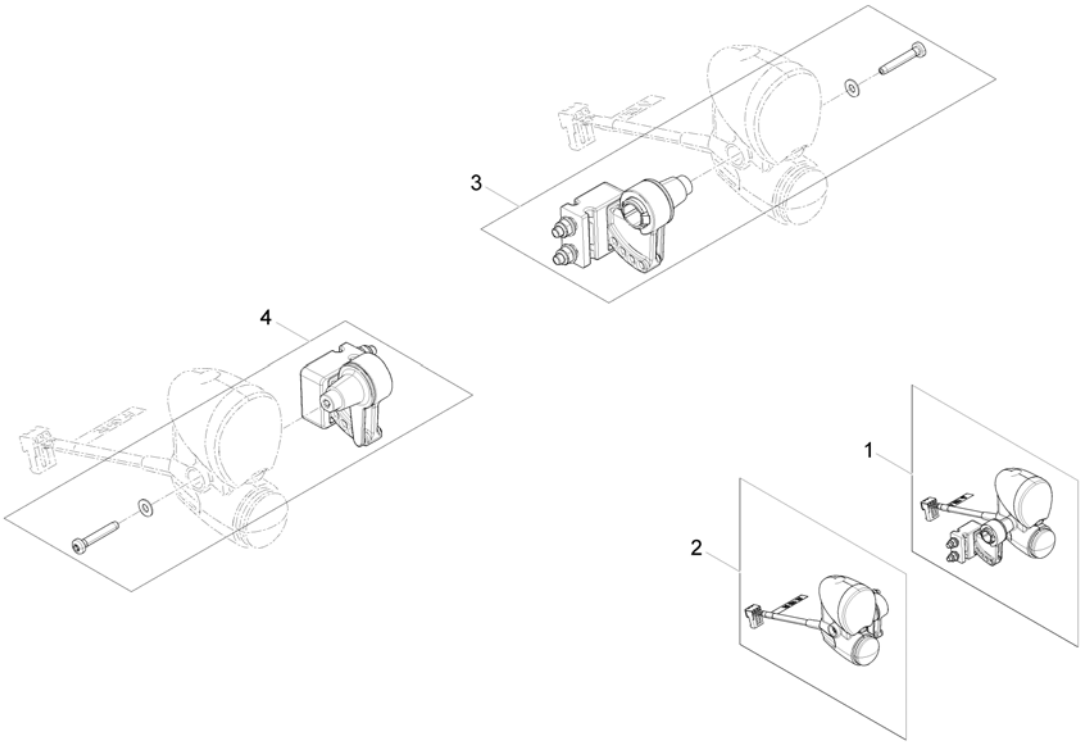

Pos.	Part number	Description	Valid from → until	Qty
1	SP1536030	Front light LED, LH		1
2	SP1536029	Front light LED, RH		1
3	SP1560611	Frontlight bracket LH		1
4	SP1562245	Frontlight bracket RH		1


LED-Frontlight for Modulite

1573222 line

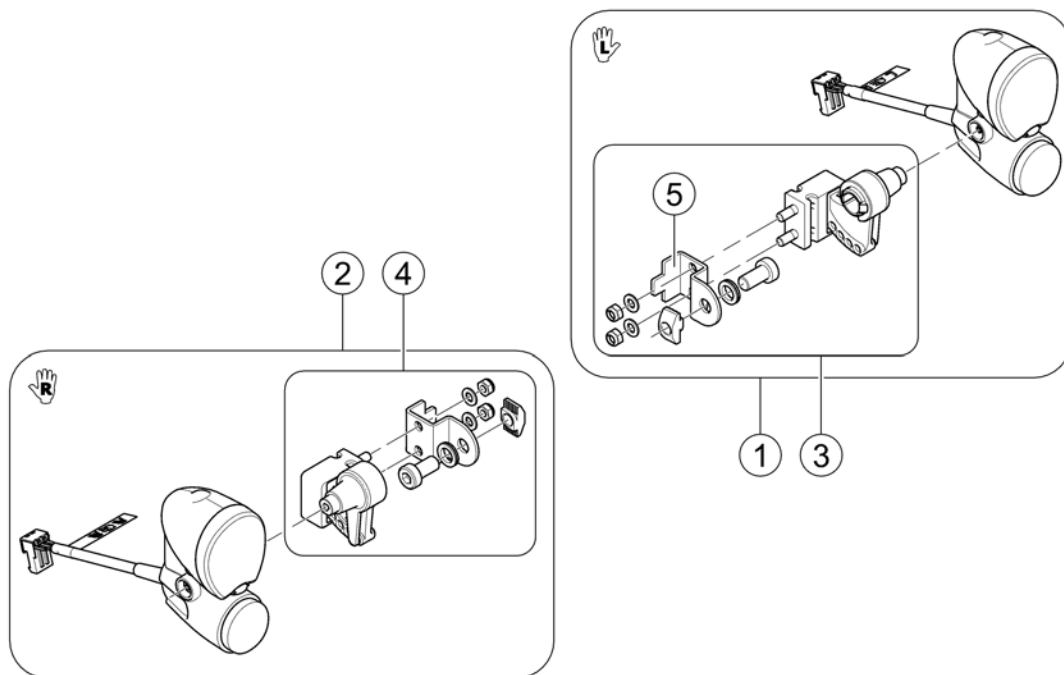

Pos.	Part number	Description	Valid from → until	Qty
-	SP1541713	Adapter cable LED light, LH <i>FDX only!</i>		1
-	SP1541654	Adapter cable LED light, RH <i>FDX only!</i>		1
-		Bracket design changed on 2015-01-10. For Retrofitting please order SP1556591.		1
1	SP1573223	LED-Frontlight LH, cpl.		1
2	SP1573224	LED-Frontlight RH, cpl.		1
3	SP1573226	Frontlight bracket LH		1
4	SP1573227	Frontlight bracket RH		1
5	SP1556591	Lamp bracket <i>New design!</i>	2015-01-10 →	2


LED-Rearlight

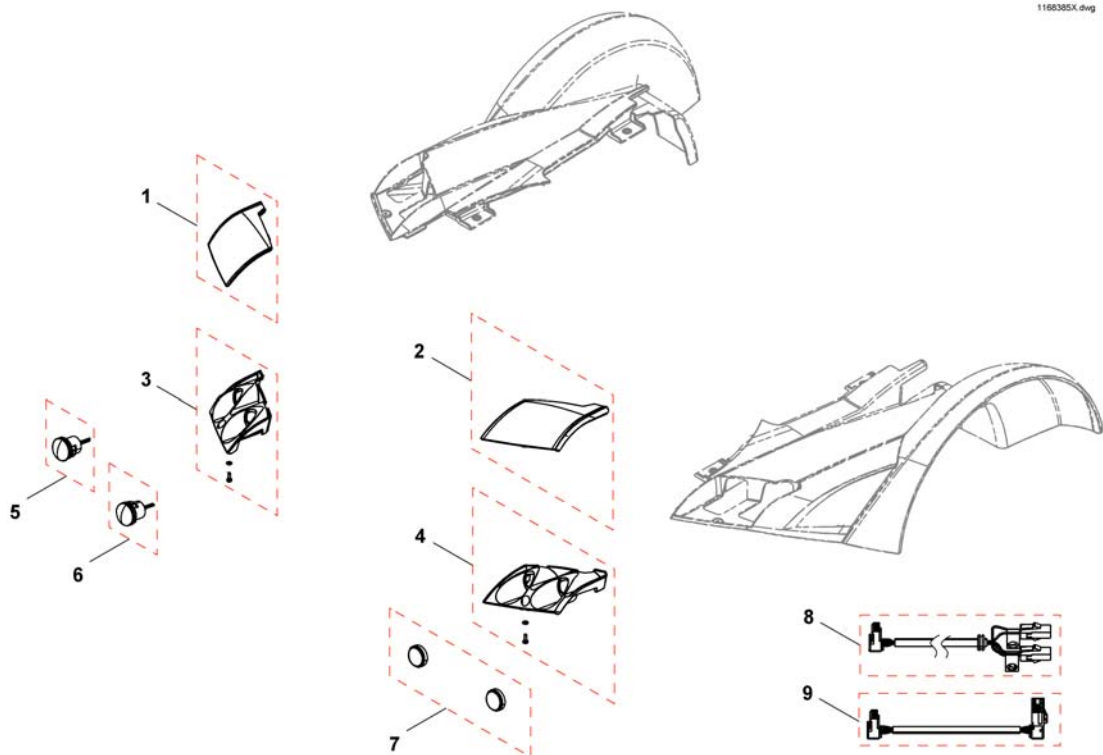

Pos.	Part number	Description	Valid from → until	Qty
1	SP1165610	Light cover LH		1
2	SP1165609	Light cover RH		1
3	SP1541748	Lamp holder LH		1
4	SP1541749	Lamp holder RH		1
5	SP1530287	Turn indicator		1
6	SP1530288	Rear light		1
7	SP1540286	Reflector-Kit (4pcs), red, Dia. 28mm		1
8	SP1529712	Rear light LED-Cable		1
9	SP1541764	No description available.		1

Rear reflectors


1541537X.dwg

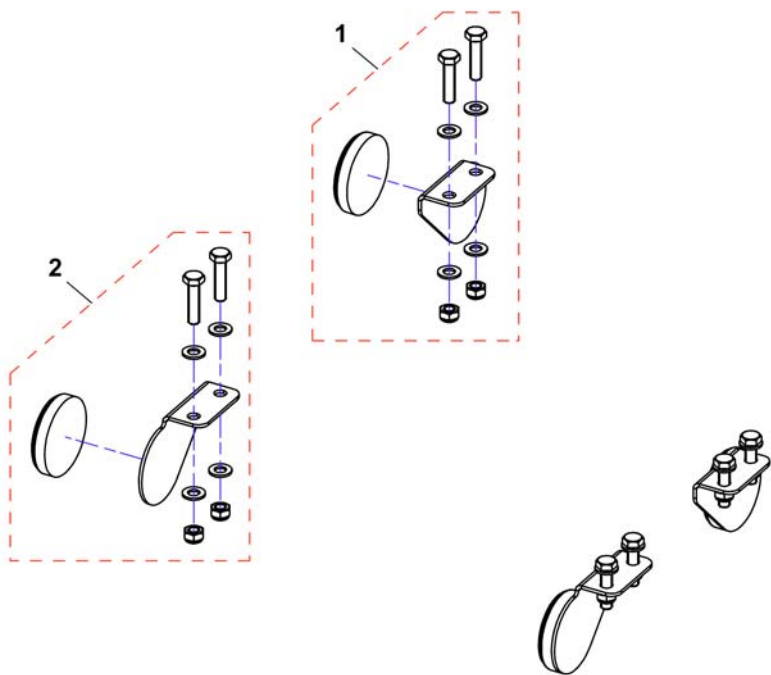

Pos.	Part number	Description	Valid from → until	Qty
1	SP1541745	Rear reflectors LH, cpl.		1
1	SP1541746	Rear reflectors RH, cpl.		1

Reflective Sign


SP1430410.kso

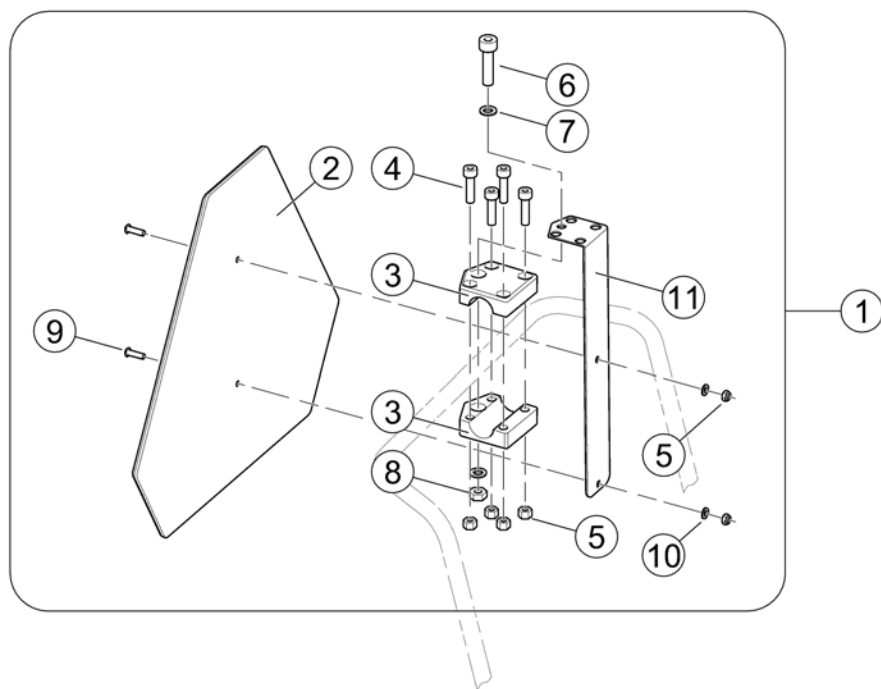

Pos.	Part number	Description	Valid from → until	Qty
1	SP1430410	Reflective Sign ECE-R 69, complete		1
2	1430834	Rear Marking Sign ECE-R 69		1
3	Z12008A.P40	Gripping clip with hexagon hole, RAL 9005 black		2
4	D50004	Cylinder Bolt M6x30-8.8 bk - DIN 912		4
5	F20109	Hexagon Nut M6 bk - DIN 985		6
6	D00102	Hexagon Bolt M8x45-8.8 bk - DIN 931		1
7	G00107	Washer Ø8,4 bk - DIN 125		2
8	-	Hexagon Nut M8 bk - DIN 985		1
9	D90187	Fillister Head Screw M6x20-10.9 bk - DIN 7380		2
10	-	Washer Ø6,4 bk - DIN 125		2
11	1430418-9005	Mounting Bracket for Reflective Sign - RAL 9005 black		1
12	SP1580671	Bracket, Reflective Sign, cpl.		1

Adapter Kit for DAHL Docking Station


1579784.iso

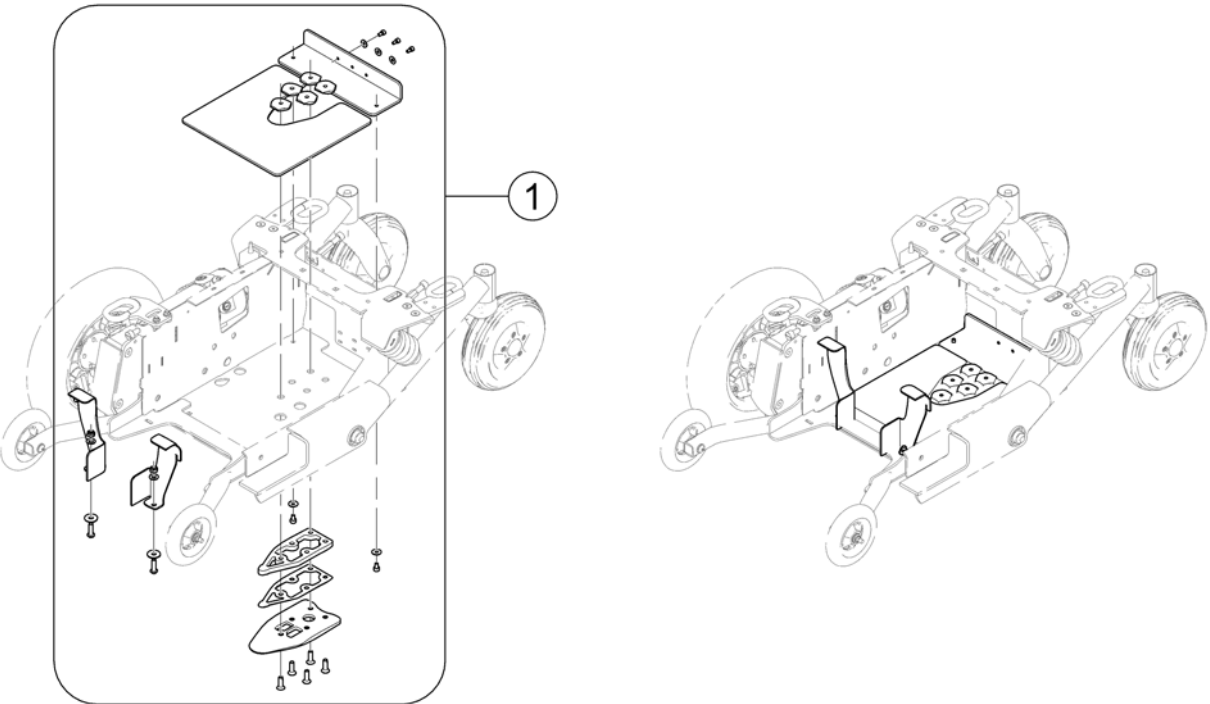

Pos.	Part number	Description	Valid from → until	Qty
1	SP1577712	Adapter Kit FDX & DAHL Docking Station		1
-	-	Backplate EasyAdapt for DAHL Docking Station <i>Needed for Preparation for DAHL Docking Station, too.</i>		1

Lap Belts / H-Belt


1529460X.dwg

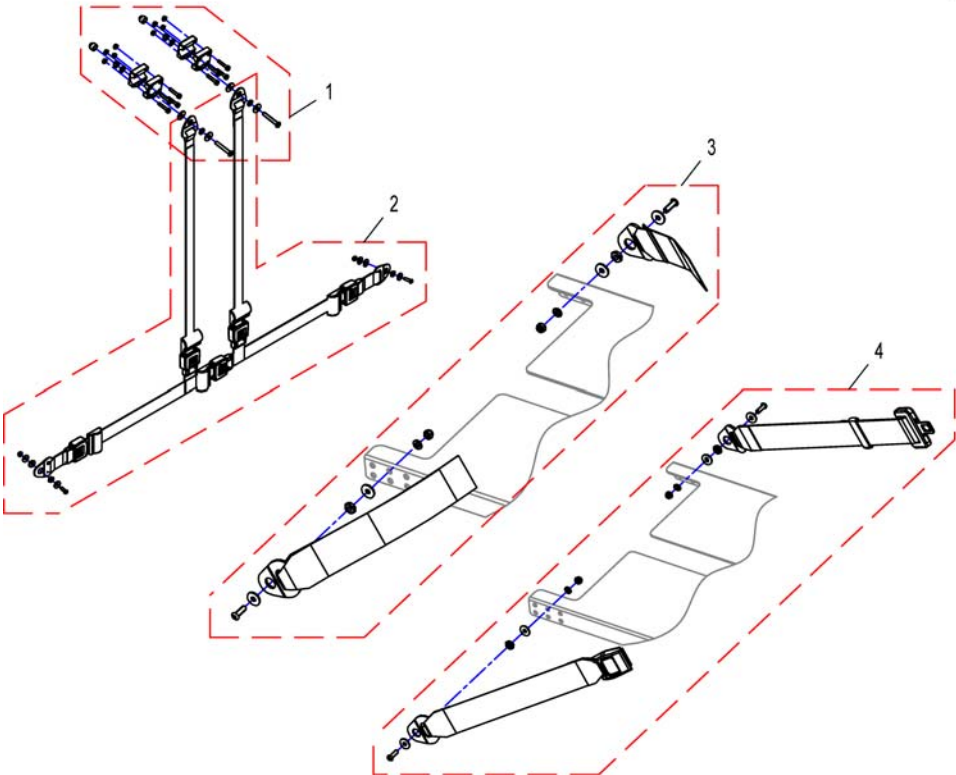

Pos.	Part number	Description	Valid from → until	Qty
1	SP1530575	Mounting-Kit H-Belt		1
2	SP1530576	H-Belt		1
3	SP1529460	Lap Belt Velcro, cpl.		1
4	SP1529461	Lap Belt Autolock, cpl.		1


Lap Belts (Modulite Seat)

1571836 iso

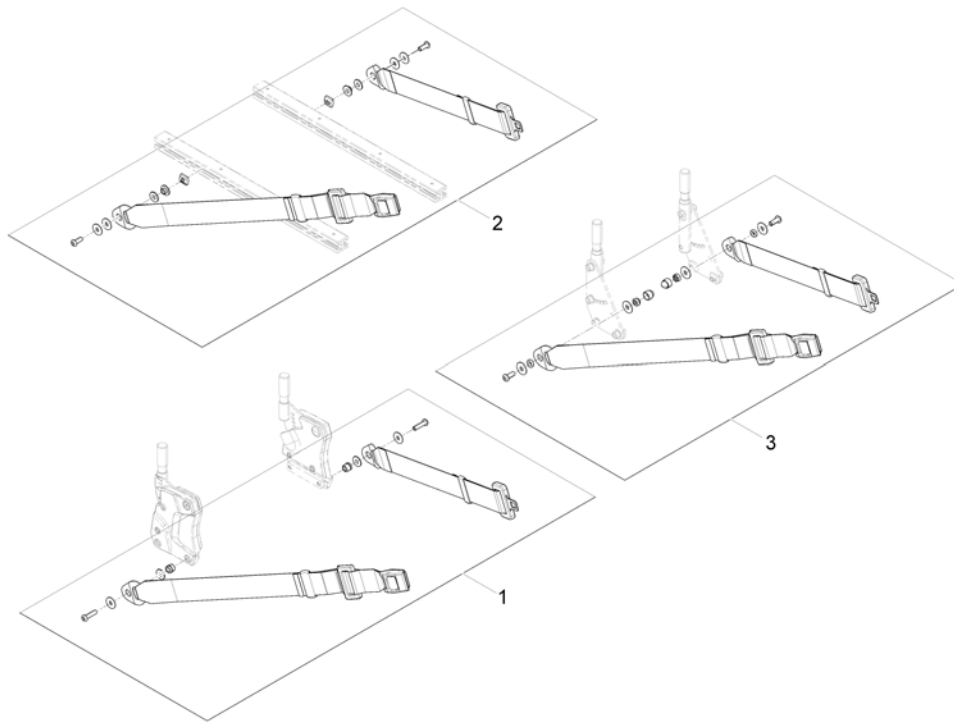


Pos.	Part number	Description	Valid from → until	Qty
1	SP1570381	Lap Belt (fix mounting on Backrest Support), cpl.		1
2	SP1571183	Lap Belt (flexible mounting in side rails), cpl.		1
3	1422845	Lap Belt, cpl.		1

H-Belts (Modulite Seat)

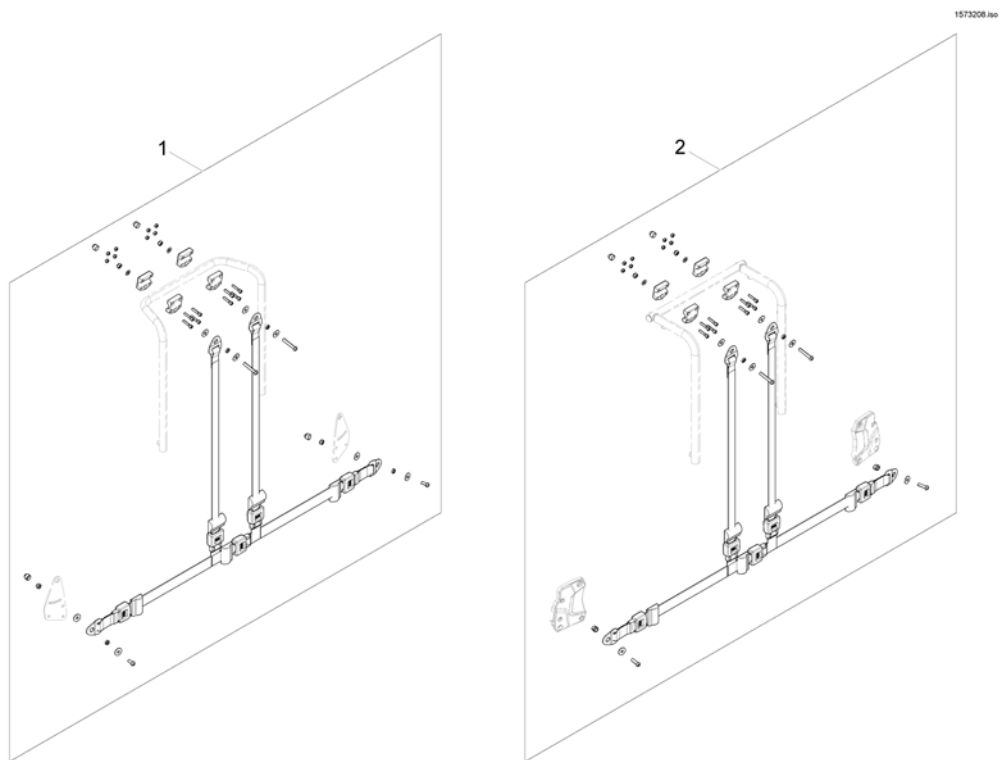

Pos.	Part number	Description	Valid from → until	Qty
1	SP1570385	H-Belt for Backrest Frame		1
2	SP1570380	H-Belt for Back Posts		1

Rearview Mirror

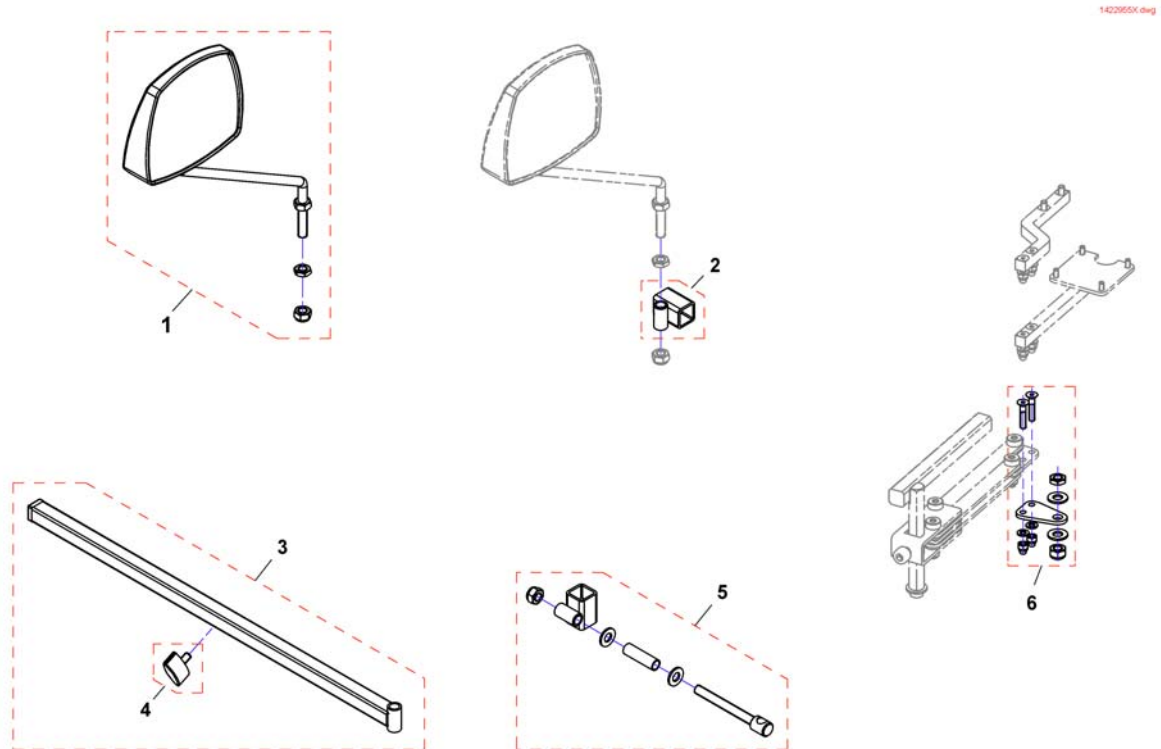


Pos.	Part number	Description	Valid from → until	Qty
1	SP1422955	Rear Mirror		1
2	F276005.00	Mirror bracket		1
3	SP1423435	Rear Mirror Support Rail cpl.		1
4	L31001	Wing ScREW M6X10		1
5	SP1422957	Mirror bracket		1
6	SP1527151	Adapter to mount mirror and remote at the same side		1

Headrests

Pos.	Part number	Description	Valid from → until	Qty
-		Headrest with cheek support [1427]		1
-		Headrest & Bracket [1350]		1
-		Headrestholder EasyAdapt Seat		1
-		Headrest, sidewise adjustable [1432]		1


Headrest with cheek support

1446560X.dwg

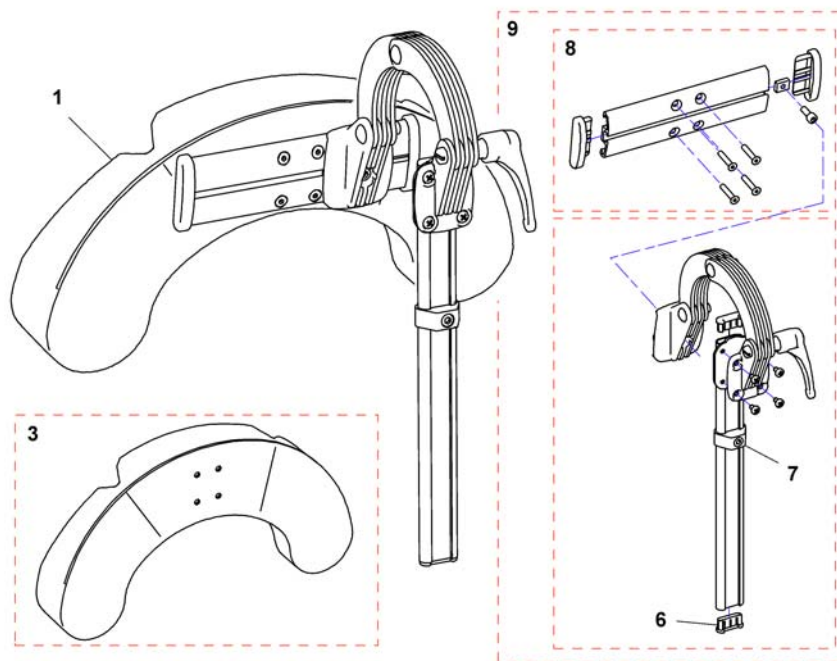

Pos.	Part number	Description	Valid from → until	Qty
1	1446560	HEAD/CHEEK SUPP. TR26 CPL. SIDEWISE ADJUSTABLE		1
1	SP1541658	Head/cheek support TR26 cpl. sidewise adjustable EasyAdapt		1
3	1444304	Cushion, cheek support		1
5	SP1529572	Headrest holder for EasyAdapt		1
6	5716497	Plug IPO "special"		1
7	1448661	stop neck and headrest		1
8	1445838	Rail head support		1
9	1485538	Headrest fixation kit		1


Headrest & Bracket

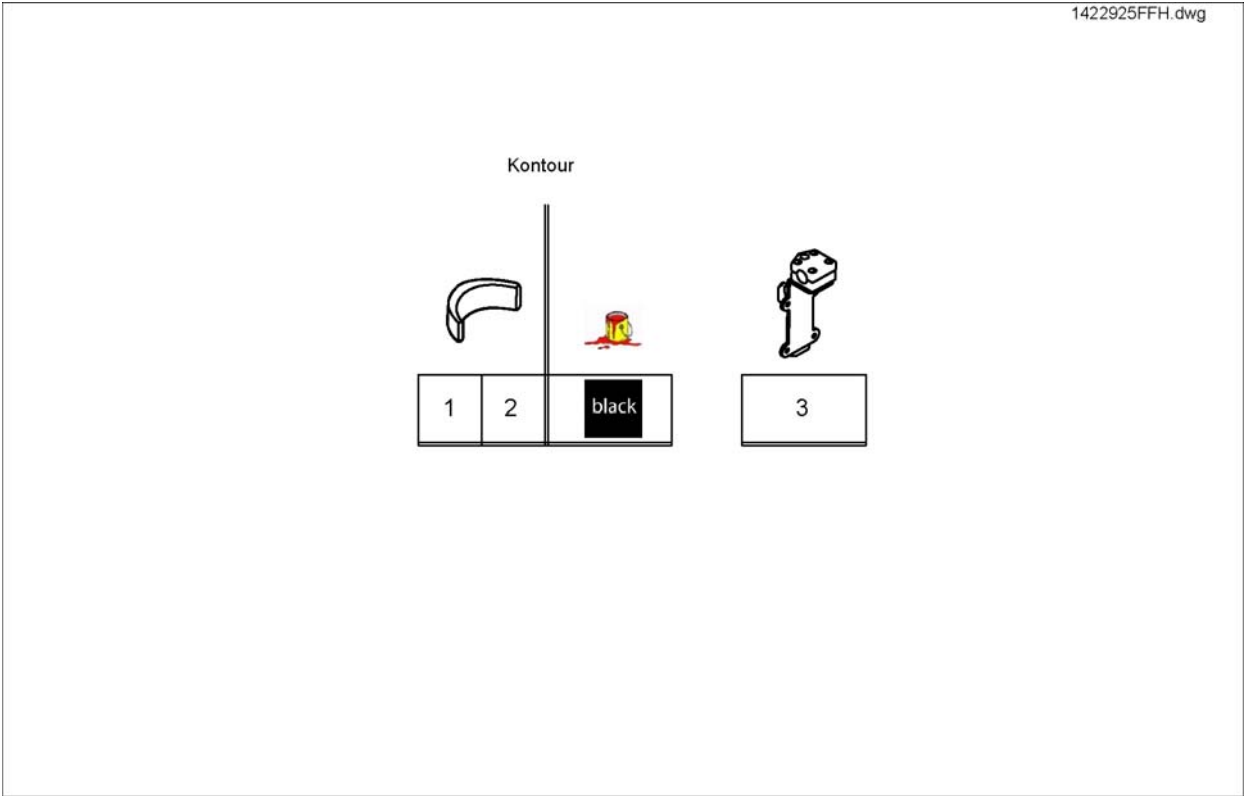

Pos.	Part number	Description	Valid from → until	Qty
1	SP1422925	Headrest, standard/contour seat, black, complete		1
2	SP1422865	Headrest cushion contour black, without bracket		1
3	SP1422914	Head-/Neckrest Bracket, cpl.		1

Headrestholder EasyAdapt Seat


1529463X.dwg

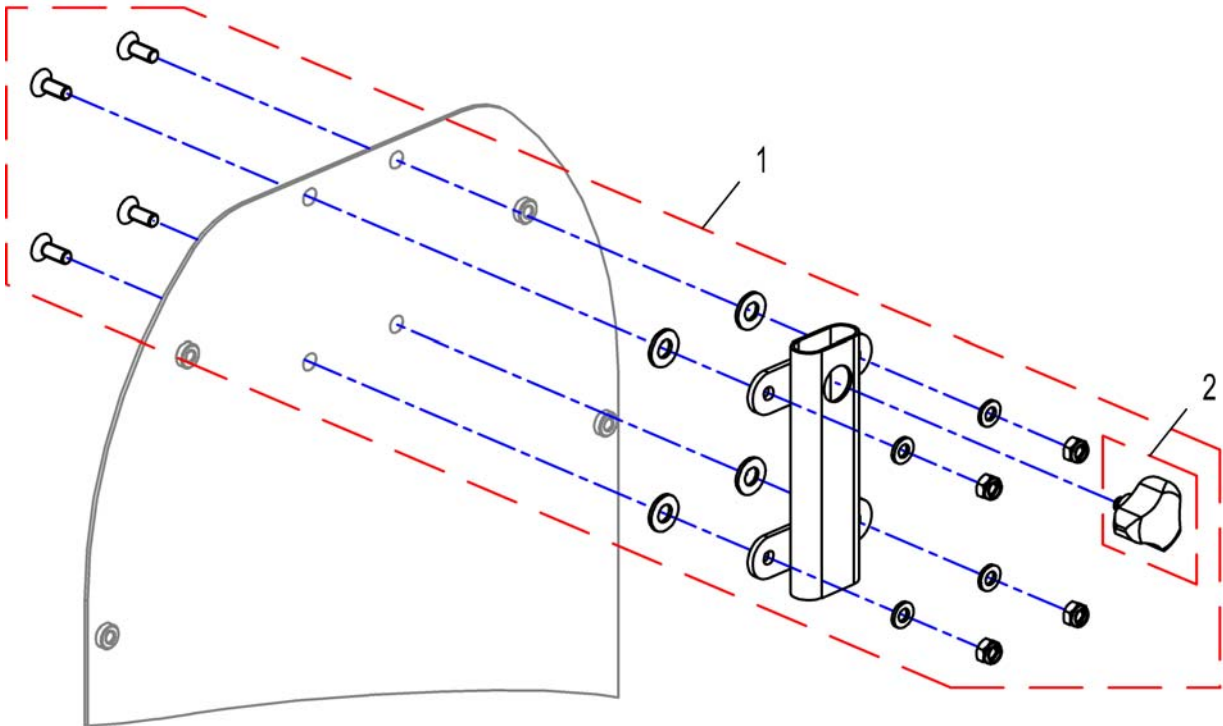

Pos.	Part number	Description	Valid from → until	Qty
1	SP1529463	Headrestholder EasyAdapt Seat, cpl.		1
2	1434623	Ball Screw Court 75mm		1

Headrest, sidewise adjustable

1530911X.no

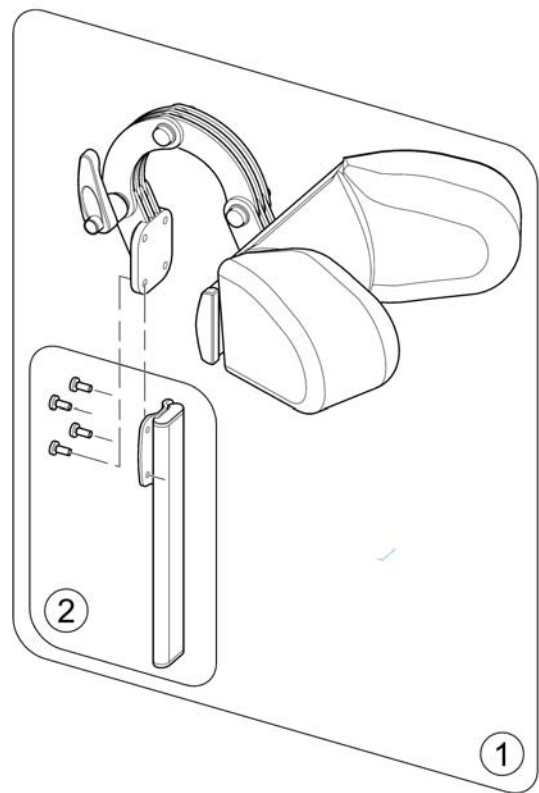


Pos.	Part number	Description	Valid from → until	Qty
1	SP1541657	Headrest, sidewise adjustable, cpl. - for EasyAdapt		1
2	SP1530567	Holder, cpl.		1

Lateral Supports

Pos.	Part number	Description	Valid from → until	Qty
-		Lateral Support - Modulite Seat		1
-		Lateral Support - EasyAdapt Seat		1

Lateral Support - Modulite Seat


Pos.	Part number	Description	Valid from → until	Qty
1	-	Lateral Support (Dartex) [1446]		1
2	-	Lateral Support, extended arm (Dartex) [1445]		1

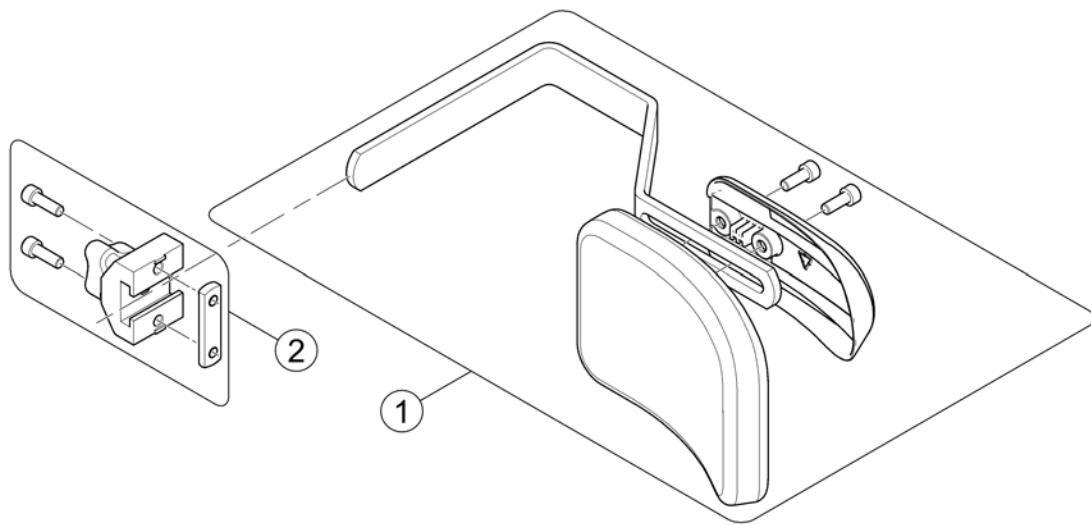

Pos.	Part number	Description	Valid from → until	Qty
-		Design change on 01. July 2015 - we suggest to change older Supports on both sides.		0
1	SP1588314	Lateral Support, cpl.		1
2	1543574	Locking trunk support		1

Lateral Support, extended arm (Dartex)

1801913.iss

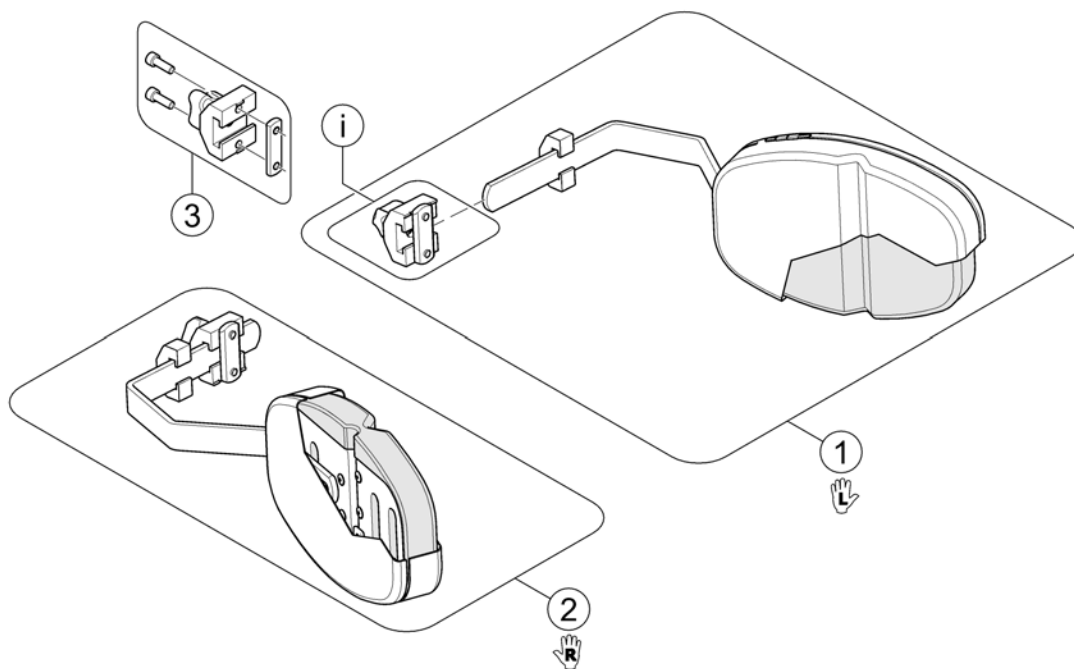


Pos.	Part number	Description	Valid from → until	Qty
1	1448692	Lateral Support, multi adjustable - LH <i>Dartex grey</i>		1
2	1448693	Lateral Support, multi adjustable - RH <i>Dartex grey</i>		1
3	1543574	Locking trunk support <i>Only for Modulite Seat!</i>		1
i	-	Not used for Flex3 Backrest!		0

Lateral Support - EasyAdapt Seat

Pos.	Part number	Description	Valid from → until	Qty
1	-	Lateral Support (Dartex) [1446]		1
2	-	Lateral Support, multi adjustable with flexible pad [1445]		1

Lateral Support (Dartex)


1570269 iso

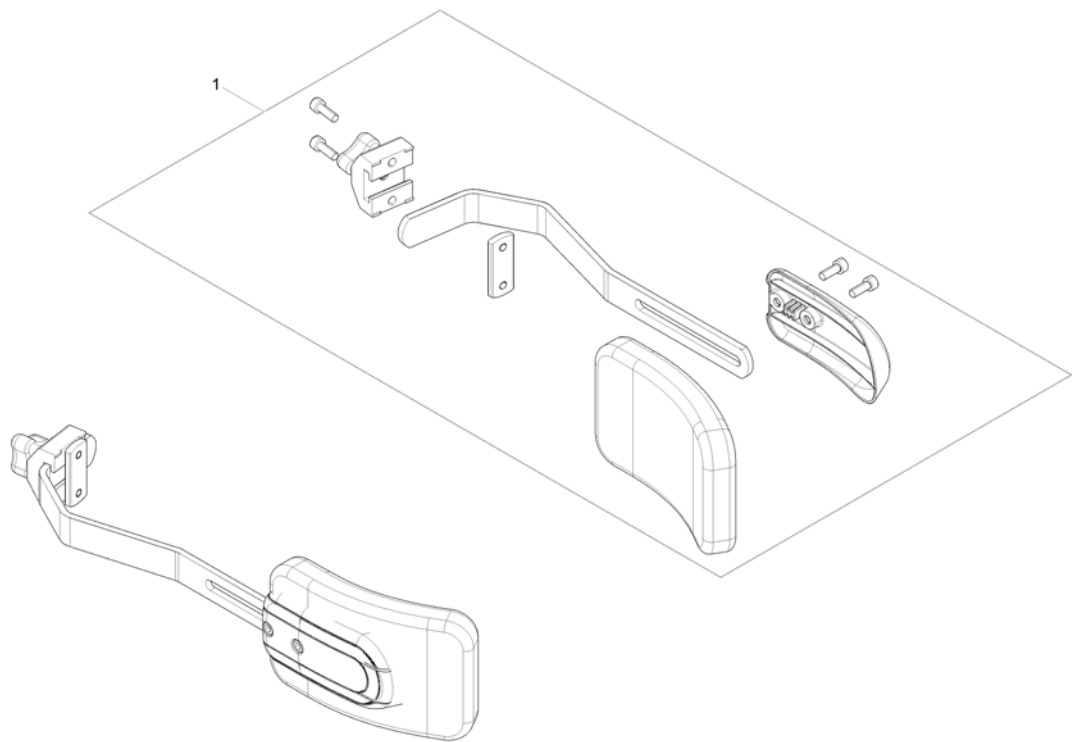

Pos.	Part number	Description	Valid from → until	Qty
	SP1561878	Trunk support, cpl.		1


Lateral Support, multi adjustable with flexible pad

1486065 iso

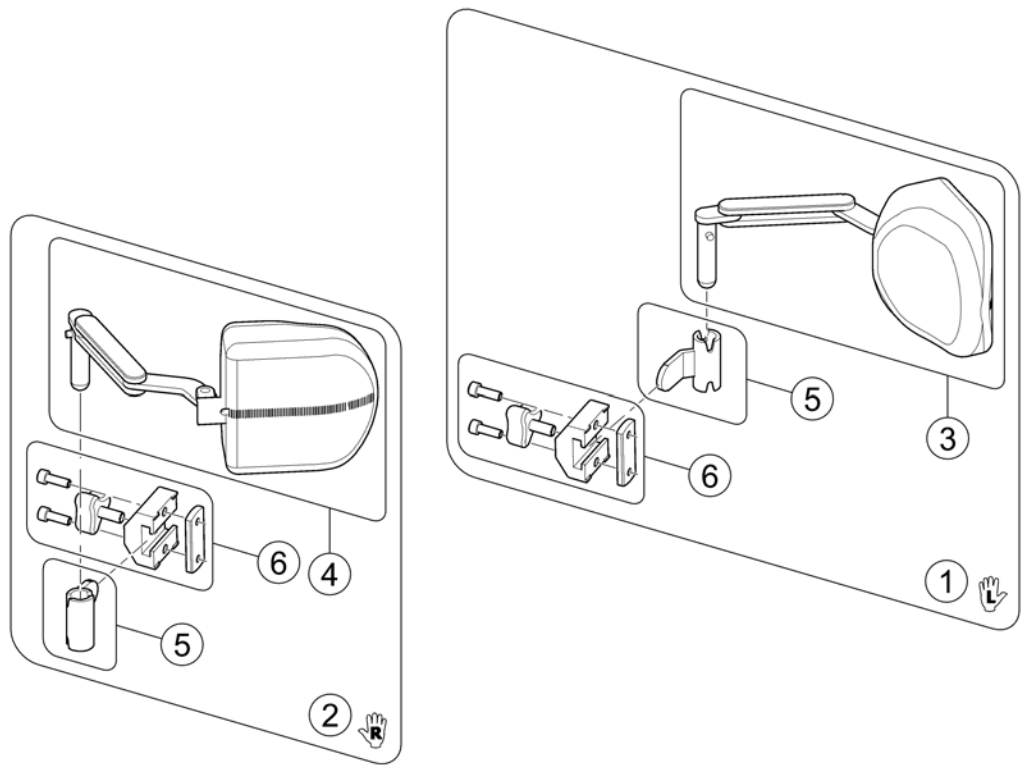

Pos.	Part number	Description	Valid from → until	Qty
1	SP1486065	Lateral Support, cpl. - LH		1
2	SP1486066	Lateral Support, cpl. - RH		1
3	1451822	Lateral support, multi adjustable - LH		1
4	1451821	Lateral support, multi adjustable - RH		1
5	1524903	Bracket LH/RH		1
6	5253170	Lateral Support Bracket		1

Bag

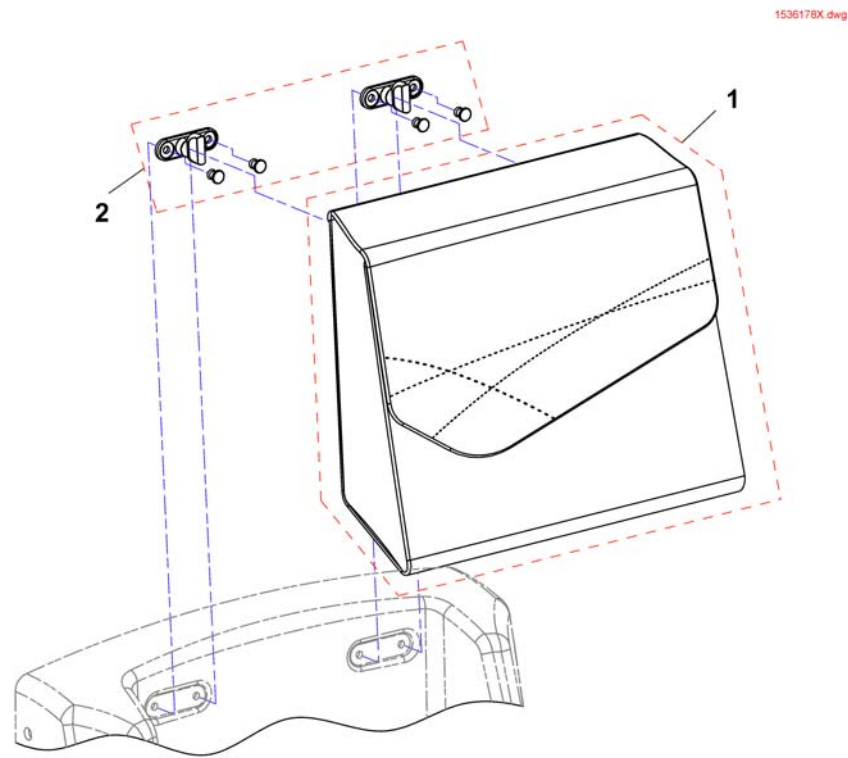


Pos.	Part number	Description	Valid from → until	Qty
1	SP1536178	Bag		1
2	SP1537708	Mounting-Kit		1

Caneholder

Pos.	Part number	Description	Valid from → until	Qty
-		Caneholder for Standard Seat (with fixed seat frame)		1
-		Cane holder (Easy Adapt Seat & Std-Seat)		1
-		Cane holder (Recaro Seat)		1
-		Cane holder (Modulite: backrest frame)		1
-		Cane holder (Modulite: back posts man recliner)		1
-		Cane holder (Modulite: back posts pow recliner)		1

Caneholder for Standard Seat (with fixed seat frame)


1567105.iso

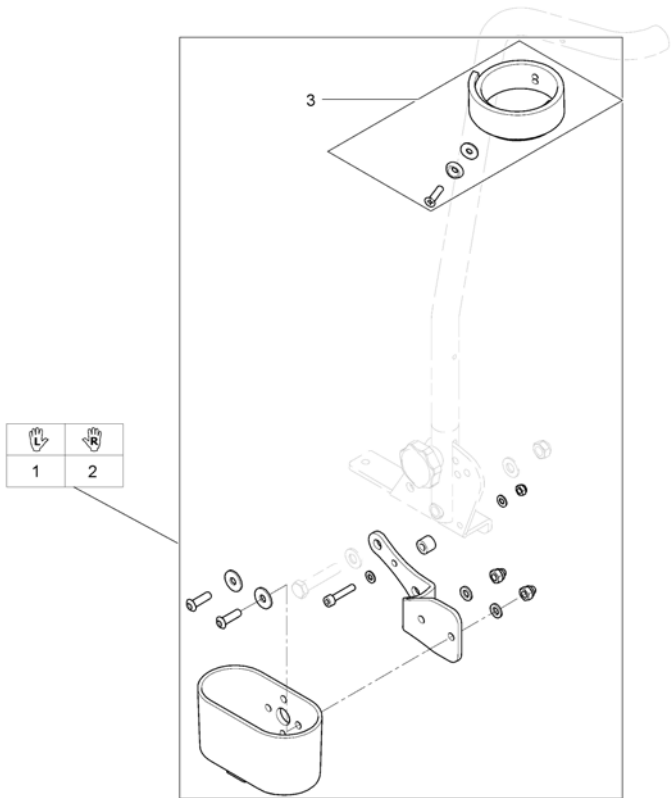

Pos.	Part number	Description	Valid from → until	Qty
1	SP1422844	Caneholder Std. cpl.		1
2	SP1534877	Cane Holder for Standard Seat, cpl.		1
3	SP1420493	Belt for caneholder		1

Cane holder (Easy Adapt Seat & Std-Seat)


1567015.iso

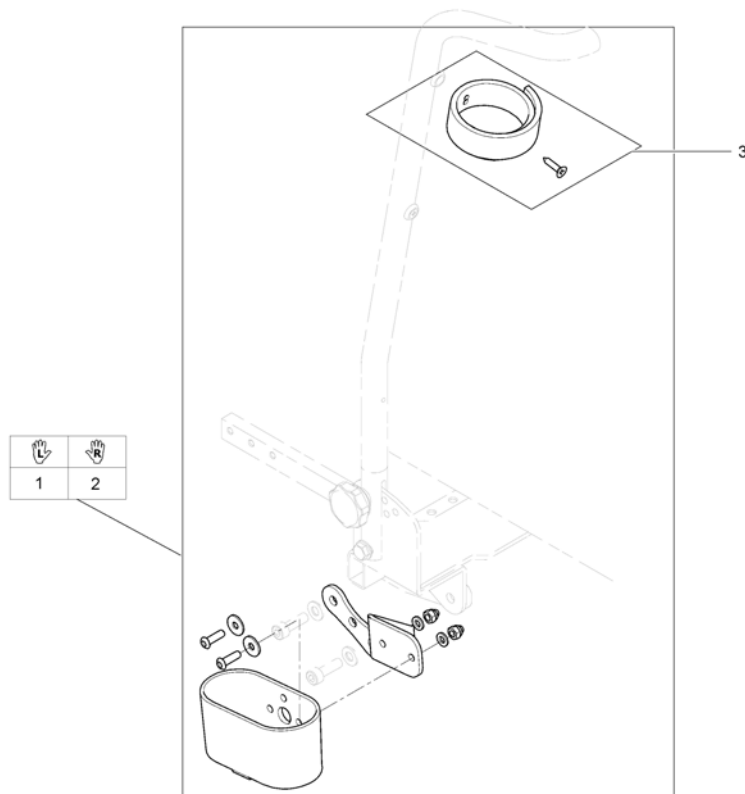

Pos.	Part number	Description	Valid from → until	Qty
1	SP1528444	Caneholder,general, cpl.		1
2	SP1529004	Caneholder,general, cpl.		1
3	SP1420493	Belt for caneholder		1


Cane holder (Recaro Seat)

1567145.iso

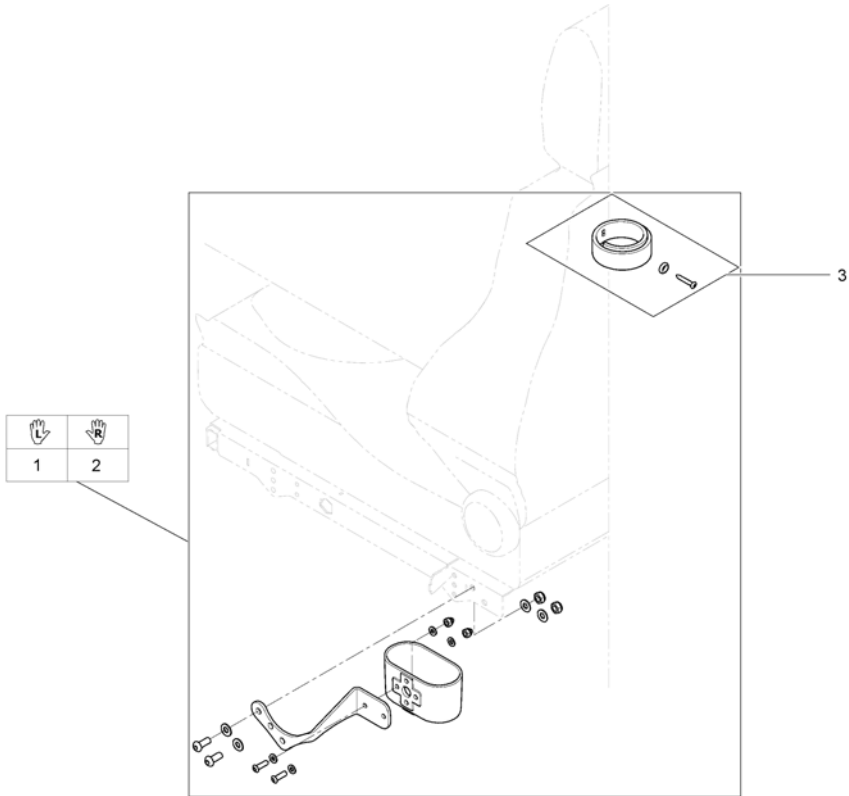


Pos.	Part number	Description	Valid from → until	Qty
1	SP1529535	Caneholder Recaro cpl		1
2	SP1529256	Caneholder Recaro cpl		1
3	SP1420493	Belt for caneholder		1

Cane holder (Modulite: backrest frame)

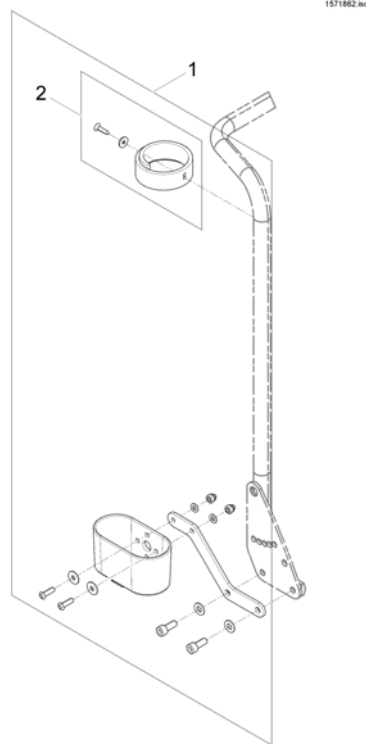


Pos.	Part number	Description	Valid from → until	Qty
1	SP1570396	Cane holder, standard backrest		1
2	SP1420493	Belt for caneholder		1

Cane holder (Modulite: back posts man recliner)

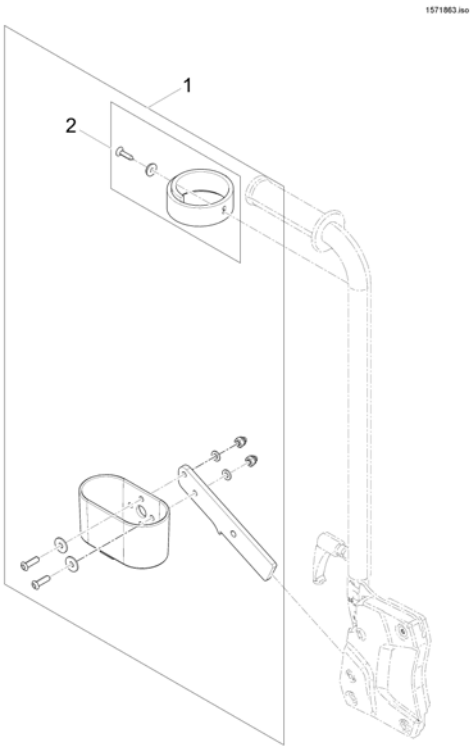


Pos.	Part number	Description	Valid from → until	Qty
1	SP1570443	Cane holder, manual backrest		1
2	SP1420493	Belt for caneholder		1

Cane holder (Modulite: back posts pow recliner)

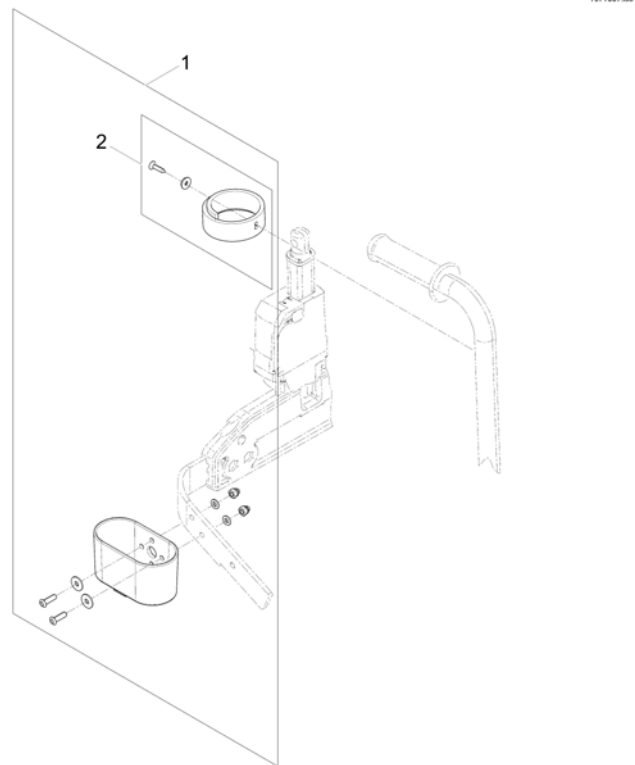

Pos.	Part number	Description	Valid from → until	Qty
1	SP1570382	Cane holder, powered backrest		1
2	SP1420493	Belt for caneholder		1

Trays

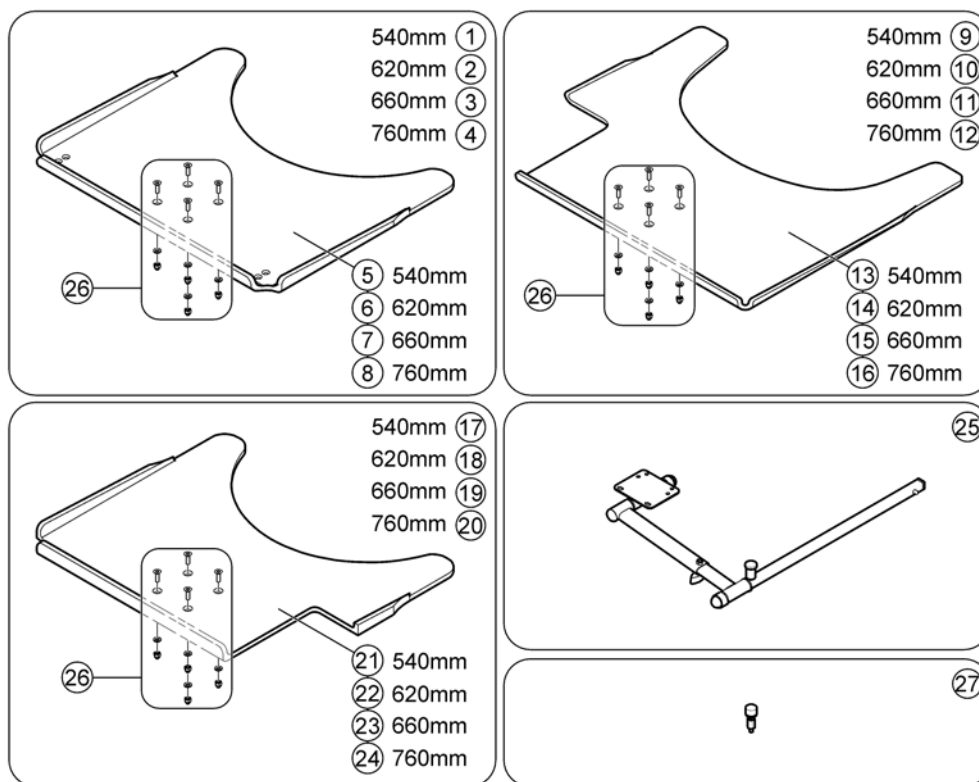


1586048.jpg


Pos.	Part number	Description	Valid from → until	Qty
1	SP1548198	Swing-Out Tray, 540mm, Seatwidth 330mm to 430mm, no Cutout, cpl. with holder		1
2	SP1548201	Swing-Out Tray, 620mm, Seatwidth 380mm to 480mm, no Cutout, cpl. with holder		1
3	SP1422954	Swing-Out Tray, 660mm, Seatwidth 430mm to 530mm, no Cutout, cpl. with holder		1
4	SP1584644	Swing-Out Tray, 760mm, Seatwidth 480mm to 580mm, no Cutout, cpl. with holder		1
5	SP1535050	Tray table, 540mm, no Cutout		1
6	SP1535010	Tray table, 620mm, no Cutout		1
7	SP1535011	Tray table, 660mm, no Cutout		1
8	SP1586049	Tray table, 760mm, no Cutout		1
9	SP1548138	Swing-Out Tray, 540mm, Seatwidth 330mm to 430mm, Cutout RH, cpl. with holder		1
10	SP1548199	Swing-Out Tray, 620mm, Seatwidth 380mm to 480mm, Cutout RH, cpl. with holder		1
11	SP1422953	Swing-Out Tray, 660mm, Seatwidth 430mm to 530mm, Cutout RH, cpl. with holder		1
12	SP1584663	Swing-Out Tray, 760mm, Seatwidth 480mm to 580mm, Cutout RH, cpl. with holder		1
13	SP1535051	Tray table, 540mm, Cutout RH		1
14	SP1535012	Tray table, 620mm, Cutout RH		1
15	SP1535049	Tray table, 660mm, Cutout RH		1
16	SP1586048	Tray table, 760mm, Cutout RH		1
17	SP1548197	Swing-Out Tray, 540mm, Seatwidth 330mm to 430mm, Cutout LH, cpl. with holder		1
18	SP1548200	Swing-Out Tray, 620mm, Seatwidth 380mm to 480mm, Cutout LH, cpl. with holder		1
19	SP1422952	Swing-Out Tray, 660mm, Seatwidth 430mm to 530mm, Cutout LH, cpl. with holder		1
20	SP1584662	Swing-Out Tray, 760mm, Seatwidth 480mm to 580mm, Cutout LH, cpl. with holder		1
21	SP1535009	Tray table, 540mm, Cutout LH		1
22	SP1534974	Tray table, 620mm, Cutout LH		1

Trays

Pos.	Part number	Description	Valid from → until	Qty
23	SP1534975	Tray table, 660mm, Cutout LH		1
24	SP1586047	Tray table, 760mm, Cutout LH		1
25	SPZ177011	Tray Support Fixing Kit cpl.SW 380/505 mm		1
26	Z177025	Fastening-Kit Tray		1
27	SP1590090	Tray Support bumper for Soft Armpad (increased height)		1

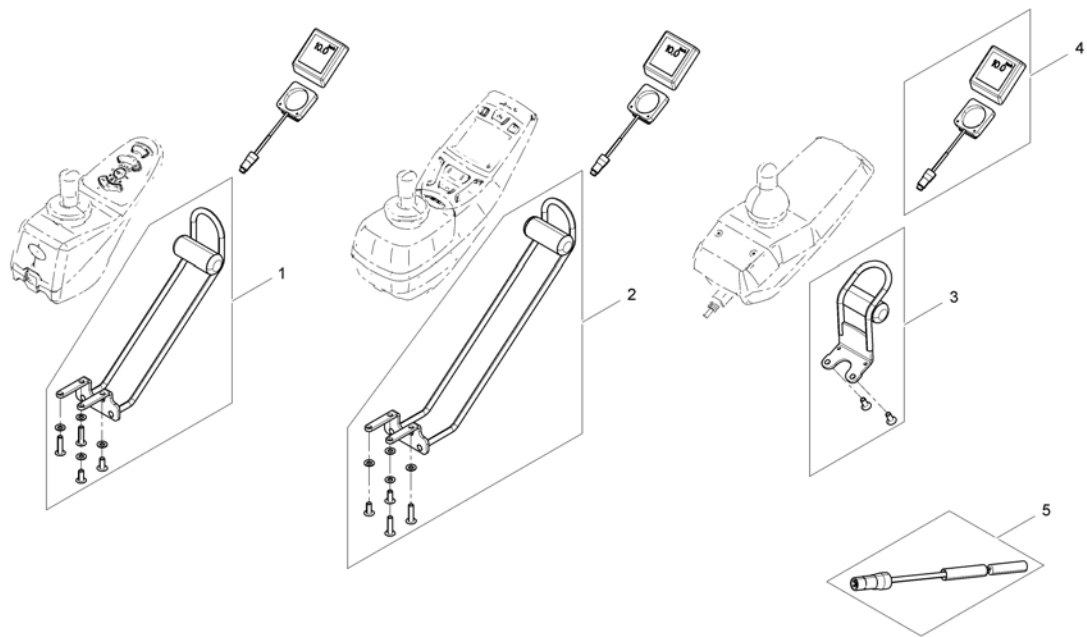


Pos.	Part number	Description	Valid from → until	Qty
1	SP1530563	Speedometer holder REM A & REM B		1
2	SP1530564	Speedometer holder REM 550		1
3	SP1546263	Speedometer holder REM24		1
4	Z183259	Speedometer		1
5	1421733	Reed-sensor cable l=2500mm		1

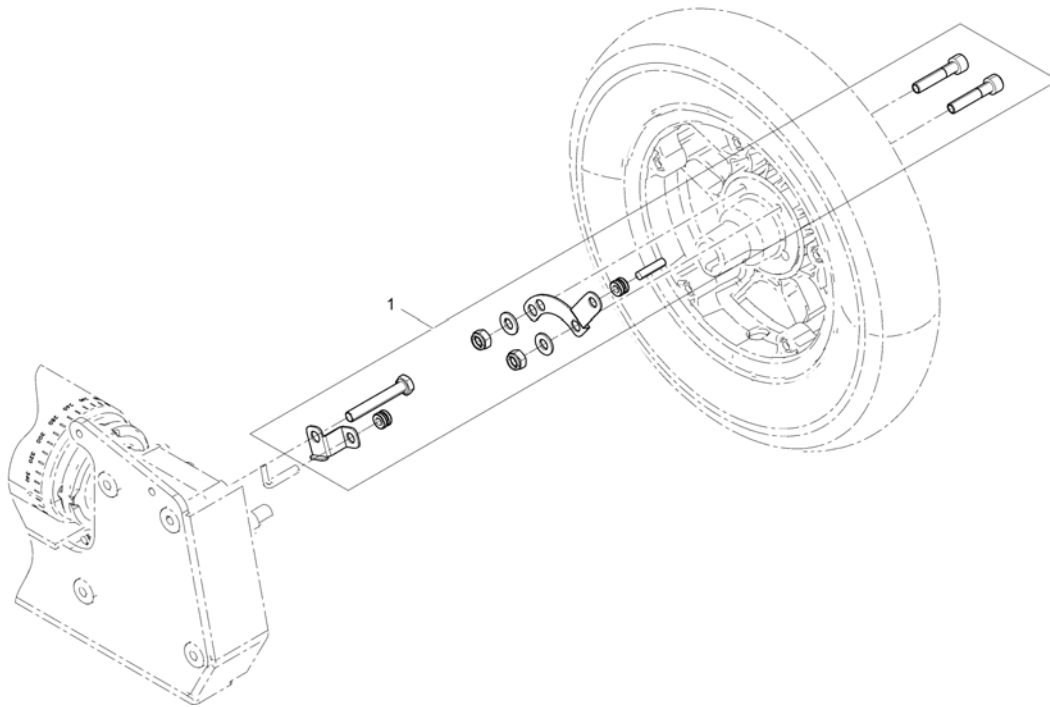


Pos.	Part number	Description	Valid from → until	Qty
1	SP1543008	Mounting-kit Speedometersensor		1

Hour Counter

1567259.iso

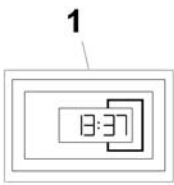


Pos.	Part number	Description	Valid from → until	Qty
-	-	Not for retrofitting - only spares!		1
1	SP1530275	Hour counter		1

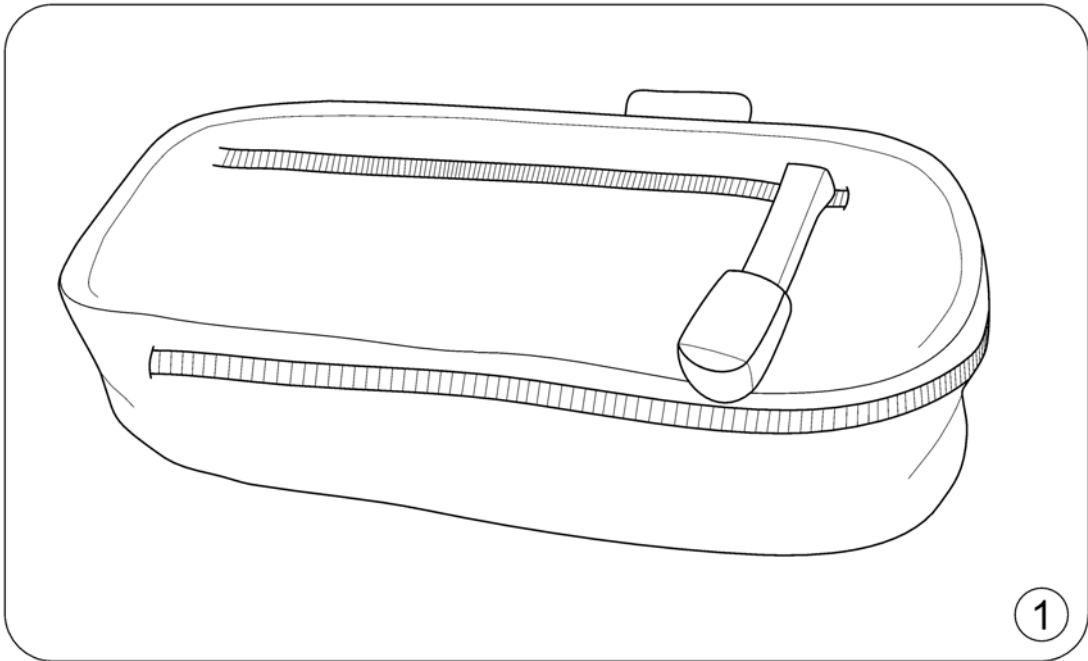

Pos.	Part number	Description	Valid from → until	Qty
1	SP1557532	KLICKfix® bag		1


KLICKfix® bracket for Fixed Armrest

1562515.iss

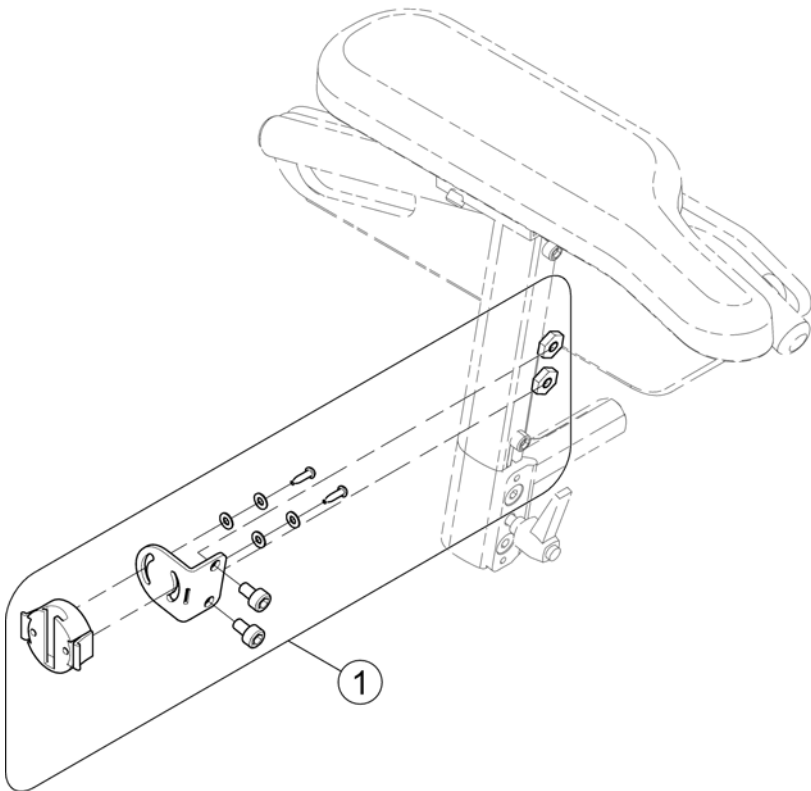

Pos.	Part number	Description	Valid from → until	Qty
1	SP1554320	KLICKfix bracket		1


1571871 iso

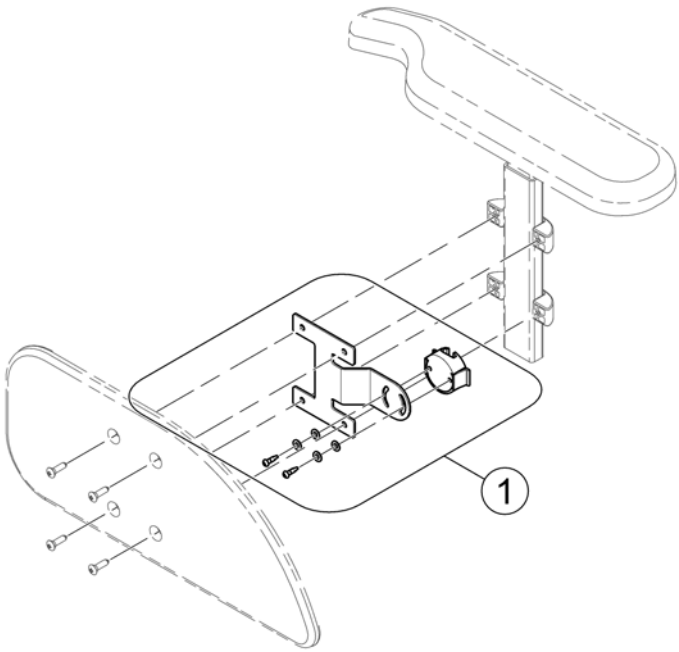

Pos.	Part number	Description	Valid from → until	Qty
1	SP1571678	KLICKfix® bracket		1

KLICKfix® bracket for Inrigged/Flip-Up/Following Armrest (Modulite)


1585167 (6)

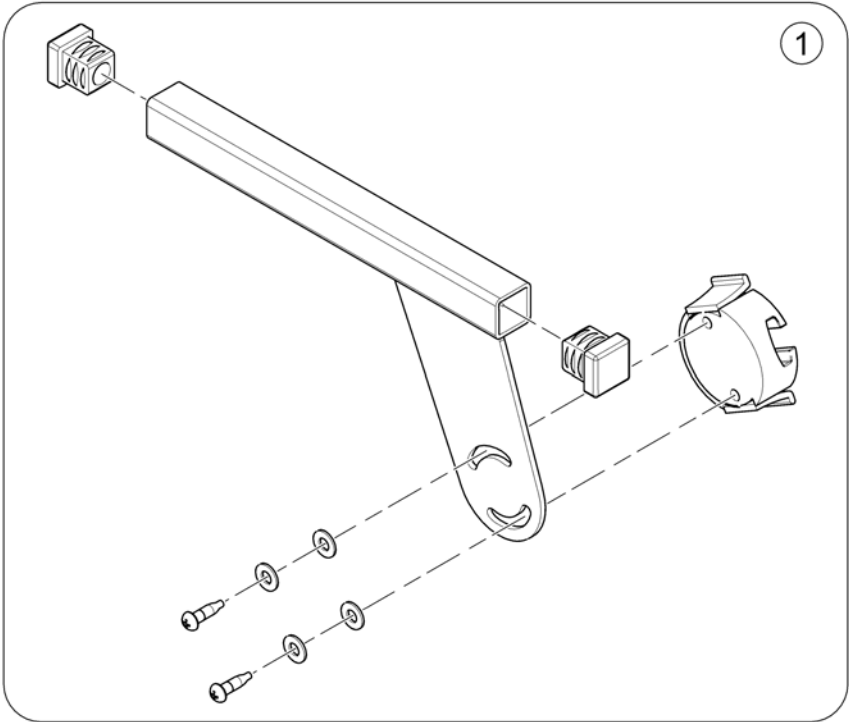

Pos.	Part number	Description	Valid from → until	Qty
1	SP1584497	KlickFix Bracket		1

Bag


1567256 iso

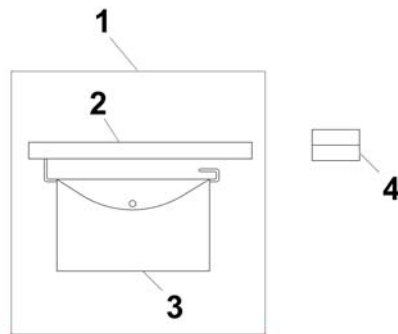

Pos.	Part number	Description	Valid from → until	Qty
1	SP1430417	Bag for side panel, cpl. LH		1
2	SP1430855	Holder for armrest bag		1
3	SP5314967	Bag for armrest		1
4	F370032-9005	Holder for side part with mirror/brake RAL 9		1

Toolkit


1567260 no

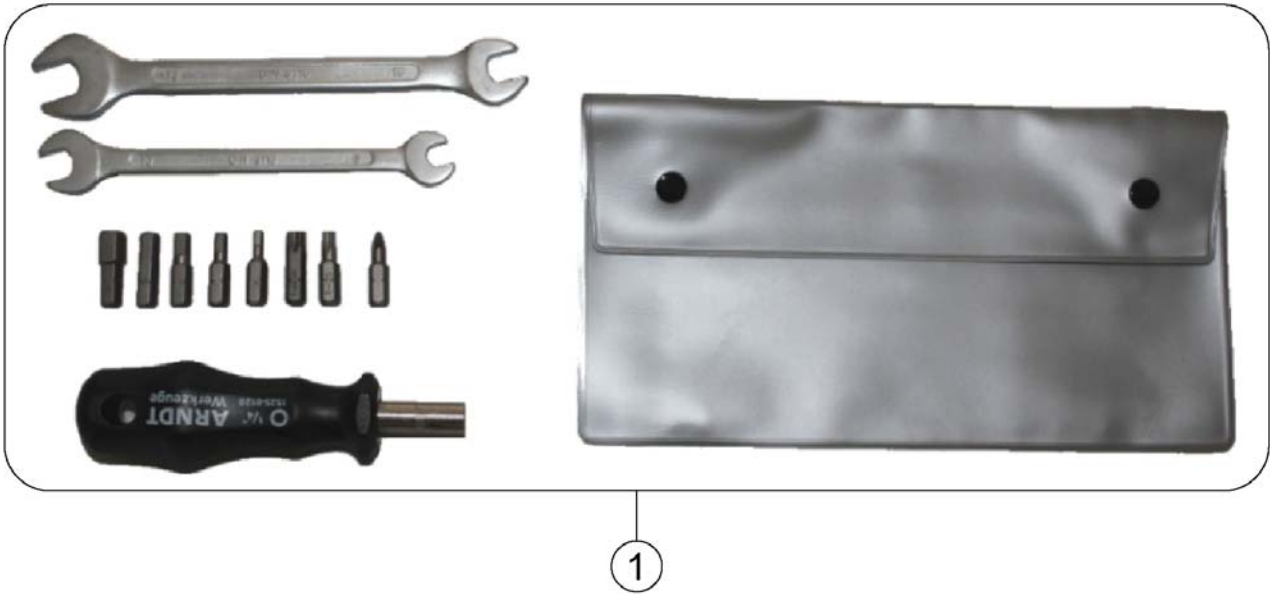

Pos.	Part number	Description	Valid from → until	Qty
1	SP1422868	Toolkit with bag		1

T-Nuts for Accessories Modulte Seat

1574067 (6)

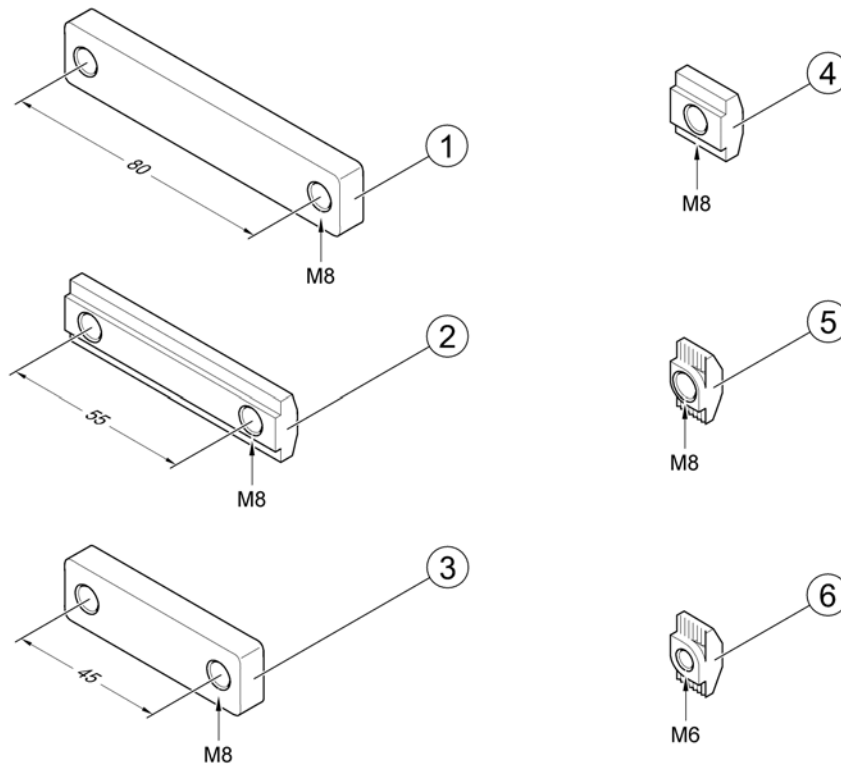

Pos.	Part number	Description	Valid from → until	Qty
1	SP1556401	T-Nut, hole distance 80mm, 2x M8		1
2	SP1572444	T-Nut, hole distance 55mm, 2x M8		1
3	SP1555168	T-Nut, hole distance 45mm, 2x M8		1
4	SP1557341	T-Nut, 1x M8		1
5	SP1573158	T-Nut, 1x M8		1
6	SP1573721	T-Nut, 1x M6		1



Yes, you can.®



Belgium & Luxemburg:

Invacare nv, Autobaan 22, B-8210 Loppem
Tel: (32) (0)50 83 10 10, Fax: (32) (0)50 83 10 11
belgium@invacare.com / www.invacare.be

Denmark:

Invacare A/S, Sdr. Ringvej 37, DK-2605 Brøndby
Tel: (45) (0)36 90 00 00, Fax: (45) (0)36 90 00 01
denmark@invacare.com / www.invacare.dk

Deutschland:

Invacare Aquatec GmbH, Alemannenstraße 10, D-88316 Isny
Tel: (49) (0)75 62 7 00 0, Fax: (49) (0)75 62 7 00 66
info@invacare-aquatec.com / www.invacare-aquatec.de

European Distributor Organisation:

Invacare, Kleiststraße 49, D-32457 Porta Westfalica
Tel: (49) (0)57 31 754 540, Fax: (49) (0)57 31 754 541
edo@invacare.com / www.invacare.eu.com

España:

Invacare SA, c/Areny s/n, Polígon Industrial de Celrà, E-17460 Celrà (Girona)
Tel: (34) (0)972 49 32 00, Fax: (34) (0)972 49 32 20
contactsp@invacare.com / www.invacare.es

France:

Invacare Poirier SAS, Route de St Roch, F-37230 Fondettes
Tel: (33) (0)2 47 62 64 66, Fax: (33) (0)2 47 42 12 24
contactfr@invacare.com / www.invacare.fr

Ireland:

Invacare Ireland Ltd, Unit 5 Seatown Business Campus, Seatown Road, Swords, County Dublin - Ireland
Tel: (353) 1 810 7084, Fax: (353) 1 810 7085
ireland@invacare.com / www.invacare.ie

Italia:

Invacare Mecc San s.r.l., Via dei Pini 62, I-36016 Thiene (VI)
Tel: (39) 0445 38 00 59, Fax: (39) 0445 38 00 34
italia@invacare.com / www.invacare.it

Nederland:

Invacare BV, Celsiusstraat 46, NL-6716 BZ Ede
Tel: (31) (0)318 695 757, Fax: (31) (0)318 695 758
nederland@invacare.com, csede@invacare.com / www.invacare.nl

Norge:

Invacare AS, Grensesvingen 9, Postboks 6230, Etterstad, N-0603 Oslo
Tel: (47) (0)22 57 95 00, Fax: (47) (0)22 57 95 01
norway@invacare.com, island@invacare.com / www.invacare.no

Österreich:

Invacare Austria GmbH, Herzog Odilostrasse 101, A-5310 Mondsee
Tel: (43) 6232 5535 0, Fax: (43) 6232 5535 4
info-austria@invacare.com / www.invacare.at

Portugal:

Invacare Lda, Rua Estrada Velha, 949, P-4465-784 Leça do Balio
Tel: (351) (0)225 1059 46/47, Fax: (351) (0)225 1057 39
portugal@invacare.com / www.invacare.pt

Sverige & Suomi:

Invacare AB, Fagerstagatan 9, S-163 91 Spånga
Tel: (46) (0)8 761 70 90, Fax: (46) (0)8 761 81 08
sweden@invacare.com, finland@invacare.com / www.invacare.se

Switzerland:

Invacare AG, Benkenstrasse 260, CH-4108 Witterswil
Tel: (41) (0)61 487 70 80, Fax: (41) (0)61 487 70 81
switzerland@invacare.com / www.invacare.ch

United Kingdom:

Invacare Limited, Pencoed Technology Park, Pencoed, Bridgend CF35 5AQ
Switchboard Tel: (44) (0)1656 776200, Fax: (44) (0)1656 776201
Customer services Tel: (44) (0)1656 776222, Fax: (44) (0)1656 776220
uk@invacare.com / www.invacare.co.uk