

.....

Spare Part List - INVACARE® Typhoon



Table of Contents - INVACARE Typhoon

.....	1
Warnings	5
Typhoon.....	6
Dataplate	6
Chassis.....	7
Shrouds	8
Wheel Suspension/ Wheels	9
Front chassis.....	9
Rear Chassis.....	10
Swivel Wheels	11
Driving Wheels complete	12
Driving Wheels slim	12
Driving Wheels wide	12
Driving Wheels detailed	14
Driving Wheels True Track.....	15
Crash kit	16
Driving unit	18
Brake	19
Decoupling	20
Electric Support	21
Lighting.....	22
Front Lights	22
Rear Lights.....	23
Electronic.....	24
Cables	25
battery cables.....	25
fuse cables	25
ACS-Buscables	25
Main cable harnesses	25
Legrests, central mounted	26
Hanger manual.....	27
Hanger electrical	28
Lower Legrests.....	29
Legrests, sideways mounted	30
Leg Support Carrier ==> 06/2005	30
Footrest receptacle, complete, manual 5cm 07/2005 ==>	31
Footrest receptacle, cpl., electric 5cm 07/2005 ==>	32
Footrest receptacle complete, manual 8cm 07/2005 ==>	33
Footrest receptacle, cpl., electric 8cm 07/2005 ==>	34
Hanger	35
Lower Footrests	36

Vari F+A Detailed	37
Release unit assembly kit Vari F ==> 06/2005.....	38
Upper Vari F, right 07/2005 ==>.....	39
Upper Vari F, left 07/2005 ==>	40
Release unit assembly kit Vari A ==> 06/2005	41
Upper Vari A, right 07/2005 ==>	42
Upper Vari A, left 07/2005 ==>.....	43
ADM + ADP Detailed.....	44
Release unit assembly kit ADM ==>06/2005	45
Upper ADM, right 07/2005 ==>	46
Upper ADM, left 07/2005 ==>	47
Assembly kit, hangers	48
Assembly kit, ADM sliding bush	49
Assembly kit, Trip lever + lock lever.....	50
Assembly kit, ADM adjustment	51
Release unit assembly kit ADP ==> 06/2005.....	52
Upper ADP, right 07/2005 ==>	53
Upper ADP, left 07/2005 ==>	54
Assembly kit, ADM Hangers	55
Assembly kit, ADM sliding bush	56
Assembly, Trip lever + Lock lever	57
Assembly kit ADP, Actuator Bracket.....	58
Assembly kit ADP, hoist motor.....	59
Lower Legrests Detailed	60
Calf plate holder, right 07/2005 ==>	61
Calf plate holder, left 07/2005 ==>	62
Back Unit	63
Backrest frame Standard	63
Backrest frame CONTOUR.....	64
Backrest frame FLEX	65
Backrest adjustment STANDARD	66
Backrest adjustment FLEX/CONTUR man.	67
Backrest adjustment FLEX/CONTUR el.	68
Backrest adjustment ASSEMBLY KIT, cpl.....	70
Retaining belts	71
Head rests	72
Backrest Upholstery	74
Seat unit	76
Seat Support/ Plate	76
Monting kit seat plate	79
RECARO-Seats	80
Holder for push handle RECARO cpl.....	81
Cushion	82
Arne Jensen-Seats.....	85

Seat unit (children)	87
Seat and Back Cushions	87
Seat Chushions	87
Back Cushions	88
Seat unit and side parts	91
Back Unit	93
Trunk Support/ Head Rest	94
Desk	95
Legrests	96
Release unit assembly kit Vari F ==> 06/2005	97
Side Panels	98
Standard Side Panels	98
Side Panels for low seatheight (38cm Tornado/ 40cm Typhoon)	99
Side Panels for Standard seatheight	101
Side part plates complete with side part adjustment	103
Side part receptacles complete	104
Side part receptacles single	105
Side Panels for Arne Jensen-Seats	106
Sliding Armrests	108
Lifter	109
Control Unit/ Joystickbox	110
Attendant Control	111
Central Steering	112
Chin Control Assembly Kit	113
Chin Control Dash Board	114
Control Unit support	115
Remote fixation, standard/ swing away	115
Remote fixation, for Attendant Control	116
Control Unit support tabular	117
Desk/ Tray control	118
Tray Control	119
Options	120
Documentation	122
User Manuals Typhoon	122
User Manuals REM 24 SD	123
User Manuals miscellaneous	124

Warnings

⚠ **Caution: Physical injury and damage to property! Loss of CE conformity and warranty due to modifications and mounting of privately purchased articles without authority!**

Alterations to the vehicle that are not approved by Invacare® and/or do not belong to the standard or optional equipment can

- downgrade vehicle handling to such an extent that the vehicle is no longer controllable,
- restrict functionality to such an extent that this may cause failure of electric/electronic components or become a fire hazard.

Modifications that have not been approved and the mounting of privately purchased articles will lead to the loss of CE conformity, Invacare® will not be liable in the event of physical injury or damage to property.

Expiration of vehicle warranty.

Maintain full efficiency and the safe use of your vehicle, avoid physical injury, damage to property or the loss of CE conformity and warranty by:

- only using original Invacare® spare parts,
- only using installation material of the same quality as the original material,
- never opening electric and electronic components (not even after warranty has expired!),
- never modifying electric, electronic or mechanical components
- never mounting parts obtained from external manufacturers without written approval from Invacare®.

✓ **Information: Definitions.**

What parts are looked upon by Invacare® as being from external manufacturers and as electric/electronic components?

Among other things, parts from external manufacturers that are subject to approval are:

- All electric and electronic components that are connected to the power supply system of the vehicle, such as seat heating, remotes, signal horns, radios.
- All mechanical mounted parts that change vehicle handling, such as trailed units and seat systems.
- In general mounted parts are not permitted that do not conform with the valid road traffic regulations and may thus also not be mounted on Invacare® vehicles.

Electric and electronic components must not be opened, e.g.:

- Electronics
- Drive motors
- Remotes
- Actuators for electric adjustments, e.g. adjustment of the back, legrests and seat angle.

⚠ **Caution: Risk of injury when assembling and dismantling the vehicle!**

Before commencing any work, ensure that the vehicle is secure. Please read and follow the valid industrial safety regulations. Carry out maintenance and service work according to the service instructions (available separately) in order to avoid physical injury and damage to property. A tool list, illustrated step-by-step instructions and details of tightening torques, will assist you and show how the vehicle remains, or will remain safe and fully serviceable.

✓ **Information: Invacare® addresses and service numbers.**

Addresses and service numbers of Invacare® branches can be found in the operating instructions, in the service instructions and on the Internet.

Dataplate



- ✓ You are often only able to find the suitable spare part by using the details on the type plate. The type plate is to be found directly on the chassis.

Please quote the serial number, model and year for every order. This helps to prevent misunderstandings.

Typenschild.dwg

The diagram shows a type plate with the following fields and callouts:

- 1** points to the **Model :** field.
- 2** points to the **SN** (Serial Number) field.
- 3** points to the **12/07/2000** production date field.

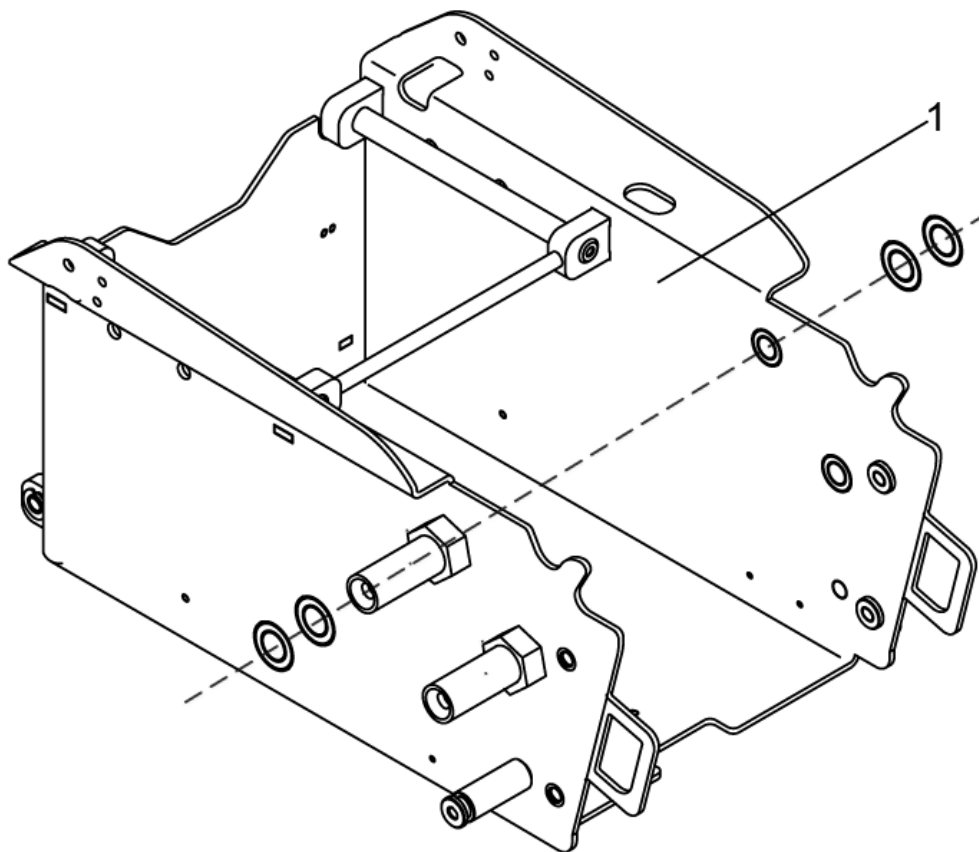
Other fields on the plate include:

- INVACARE Deutschland GmbH, Kleiststrasse 49, D-32457 Porta-Westfalica - GERMANY
- INVACARE logo
- CE mark
- Max Speed:
- Max Incline:
- Front Whl Load:
- Mid Whl Load:
- Rear Whl Load:
- Total Wt:
- Unladen Wt:
- Max User Wt:
- REF :
- Agreement No:
- EAN barcode (1234567891011212)

item	description
1	MODEL DESIGNATION
2	SERIAL NUMBER
3	PRODUCTION DATE

Chassis


1440304.dwg



item	ordercode	description	quantity
1	SP1440304	PREASSEMBLED CHASSIS TYPHOON	1


Shrouds

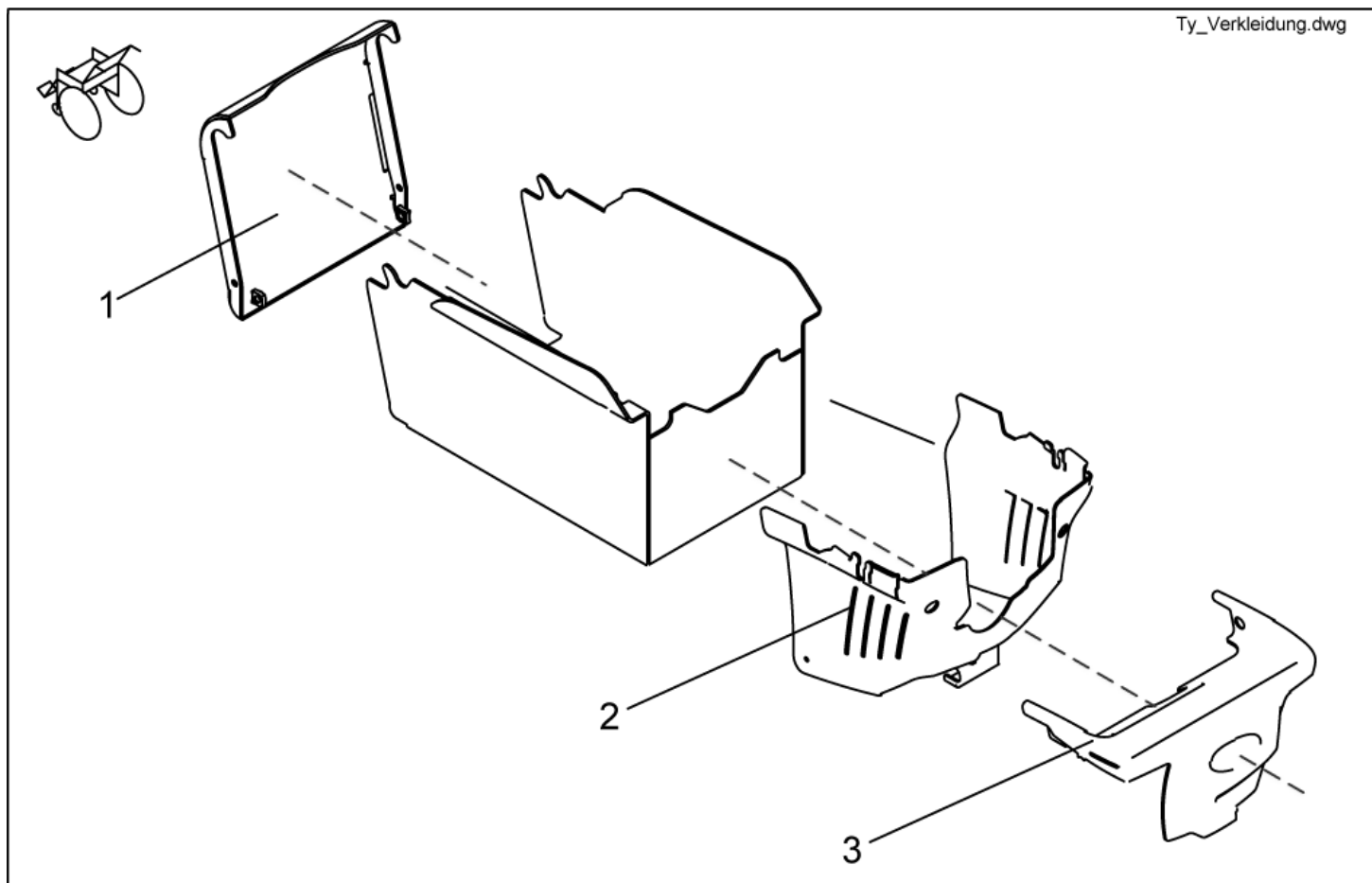Ty_Verkleidung.dwg




item	ordercode	description	quantity
1	SP1440315	FRONT COVER CPL.	1
2	SP1440328	COVER FOR ELECTRONIC MODULES WITH MOUNTING KIT	1
3	SP1440356	REAR COVER COMPLETE	1

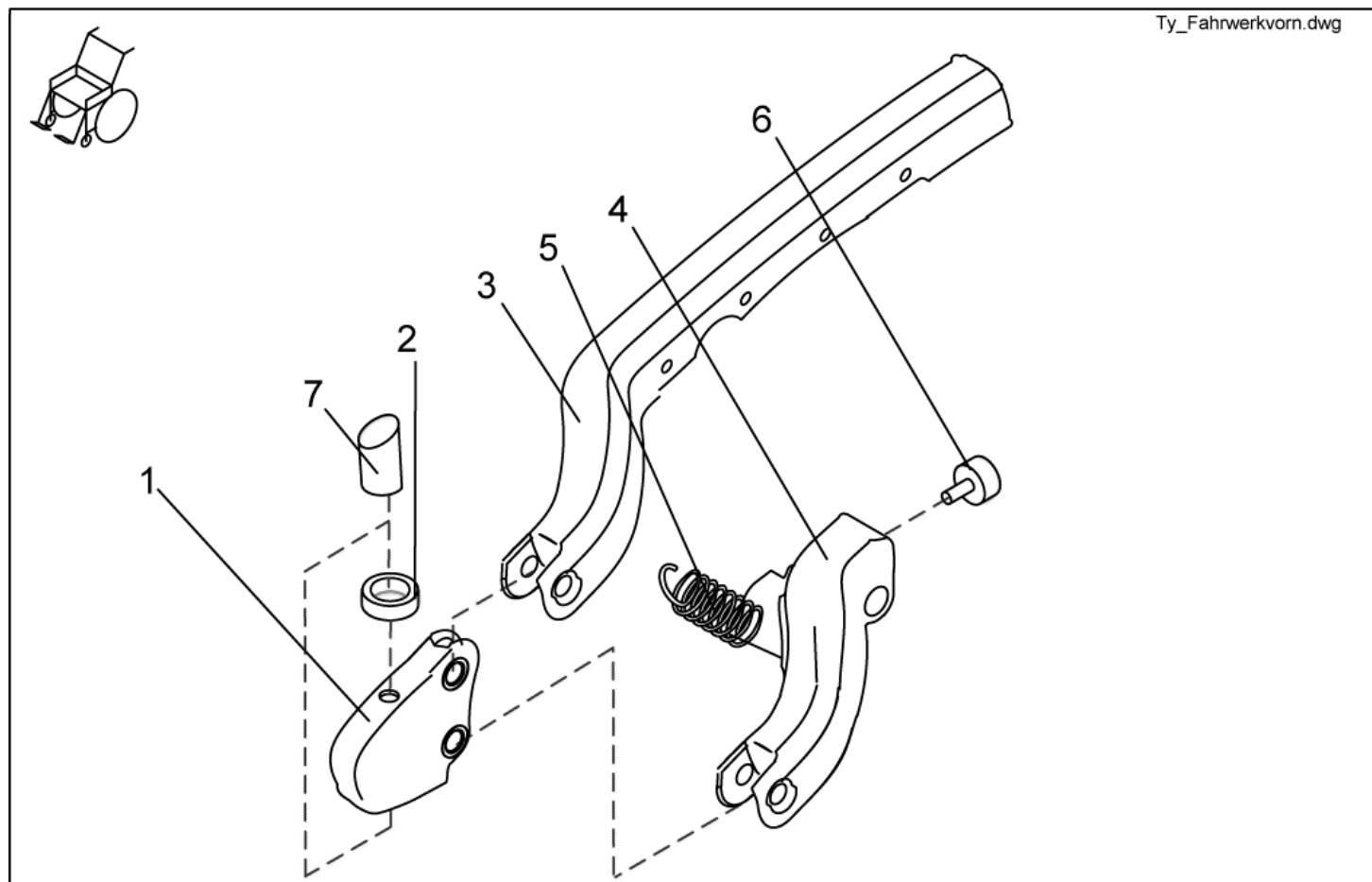Wheel Suspension/ Wheels - Front chassis




item	ordercode	description	quantity
1	1434076-5026	NOSE WALKING BEAM BLUE	2
1	1434076-2013	NOSE WALKING BEAM RED	2
1	1434076-6036	NOSE WALKING BEAM GREEN	2
-	1440571	MOUNTING KIT FOR NOSE WALKING BEAM	2
-	1440618	SET WITH BUSHES FOR WALKING BEAM	
2	1419254	ANGULAR BALL BEARING 7202B 2RS	4
3	1433762-5026	WALKING BEAM TOP BLUE LEFT	1
3	1433762-2013	WALKING BEAM TOP RED LEFT	1
3	1433762-6036	WALKING BEAM TOP GREEN LEFT	1
3	1433760-5026	WALKING BEAM TOP BLUE RIGHT	1
3	1433760-2013	WALKING BEAM TOP RED RIGHT	1
3	1433760-6036	WALKING BEAM TOP GREEN RIGHT	1
4	1433763-5026	WALKING BEAM BELOW BLUE LEFT	1
4	1433763-2013	WALKING BEAM BELOW RED LEFT	1
4	1433763-6036	WALKING BEAM BELOW GREEN LEFT	1
4	1433761-5026	WALKING BEAM BELOW BLUE RIGHT	1
4	1433761-2013	WALKING BEAM BELOW RED RIGHT	1
4	1433761-6036	WALKING BEAM BELOW GREEN RIGHT	1
5	1440710	SPRING COMPLETE	2
6	1440709	BUFFER PLATE (RUBBER) WITH NUT, PAIR	1
7	1436347	CAP	2

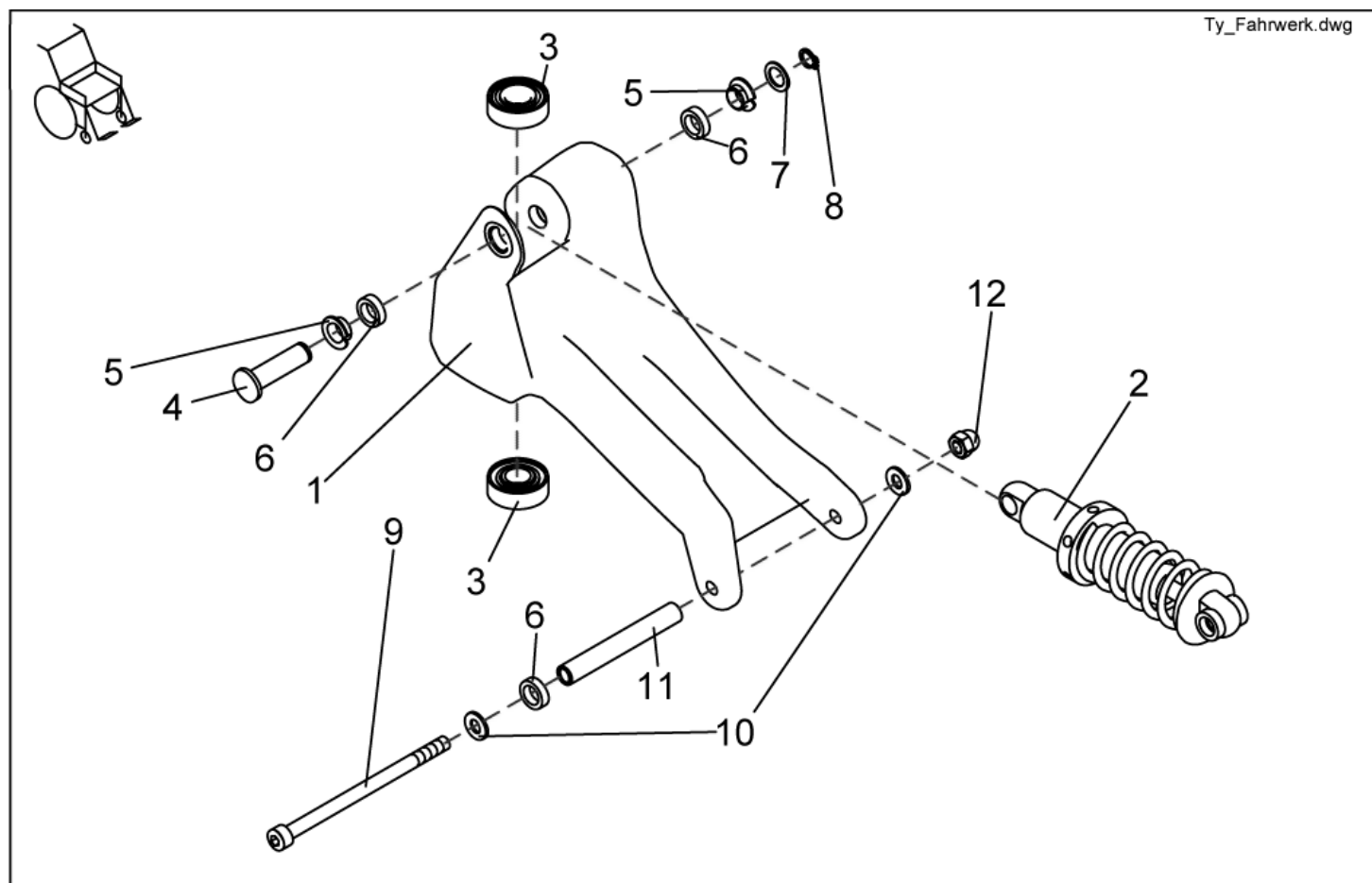Wheel Suspension/ Wheels - Rear Chassis




item	ordercode	description	quantity
1	1434078-5026	LEFT-HAND REAR SECTION, BLUE	1
1	1434078-2013	LEFT-HAND REAR SECTION, RED	1
1	1434078-6036	LEFT-HAND REAR SECTION, GREEN	1
1	1434077-5026	RIGHT-HAND REAR SECTION, BLUE	1
1	1434077-2013	RIGHT-HAND REAR SECTION, RED	1
1	1434077-6036	RIGHT-HAND REAR SECTION, GREEN	1
2	1440550	SHOCK ABSORBER DV-22-165 WITH SPRING 350LBS	2
3-12	1440572	REAR SECTION ASSEMBLY KIT	2
3	1419254	ANGULAR BALL BEARING 7202B 2RS	4
4	NA	PIVOT BUSH	1
5	NA	BUSH BB1207 DU	2
6	NA	WASHER 12,2X18X5	3
7	NA	WASHER Ø12/18X1 DIN988	1
8	NA	RETAINER RING 10 MM	1
9	NA	SCREW DIN 912 M8X130	1
10	NA	WASHER 8,4 X 17 X 1,6	2
11	NA	BUSH	1
12	NA	DOMED NYLOC NUT M8 X 1	1
-	NA	TOUCH-UP PAINT RAL 2013 ORANGE	1
-	NA	TOUCH-UP PAINT RAL 6036 GREEN	1
-	NA	TOUCH-UP PAINT RAL 5026 BLUE	1

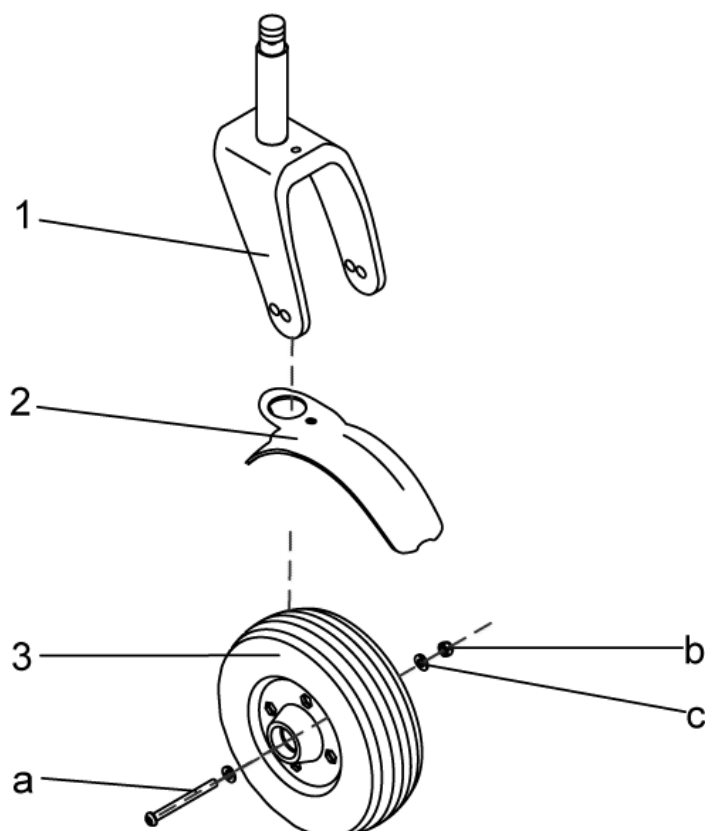

item	ordercode	description	quantity
1	1440566	SWIVEL WHEEL FORK with STEERING HEAD AXLE 108mm	2
1	1446197	SWIVEL WHEEL FORK with STEERING HEAD AXLE 92mm (for front swivel wheels)	2
2	1440565	SPLASH GUARD FRONT WITH MOUNTING KIT	2
2	1442733	SPLASH GUARD REAR WITH MOUNTING KIT	2
3	1440414	SWIVEL WHEEL	4
a-c	1440564	SWIVEL WHEEL ASSEMBLY KIT	4
a	NA	ISO 7380 M8X90 10.9 ELECTRO-GALVANISED SCREW	1
b	NA	CAP NUT M8 DIN 986	1
c	NA	WASHER 8,4 X 17 X 1,6	2

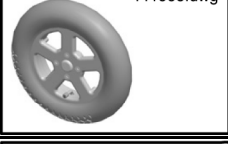
Wheel Suspension/ Wheels - Driving Wheels complete

Picture	Filling	Hub	color	ordercode
	Luft	Standard	grau	SP1440353
	Luft	Standard	Trelleborg	SP1440354
	Luft	Bremse	grau	SP1440355
	Luft	Bremse	Trelleborg	SP1441536
	pannengeschützt	Standard	grau	SP1441537
	pannengeschützt	Standard	Trelleborg	SP1441538
	pannengeschützt	Bremse	grau	SP1441539
	pannengeschützt	Bremse	Trelleborg	SP1441540

Picture	Filling	Hub	color	ordercode
	Luft	Standard	grau	SP1440469
	Luft	Standard	Trelleborg	SP1440470
	Luft	Bremse	grau	SP1440471


Picture	Filling	Hub	color	ordercode
 1440472.dwg	Luft	Bremse	Trelleborg	SP1440472
 1440473.dwg	pannengeschützt	Standard	grau	SP1440473
 1440474.dwg	pannengeschützt	Standard	Trelleborg	SP1440474
 1440475.dwg	pannengeschützt	Bremse	grau	SP1440475
 1440476.dwg	pannengeschützt	Bremse	Trelleborg	SP1440476
 1441541.dwg	pannensicher	Standard	grau	SP1441541
 1441542.dwg	pannensicher	Bremse	grau	SP1441542
	pannengeschützt	Bremse	Trelleborg	SP1439543

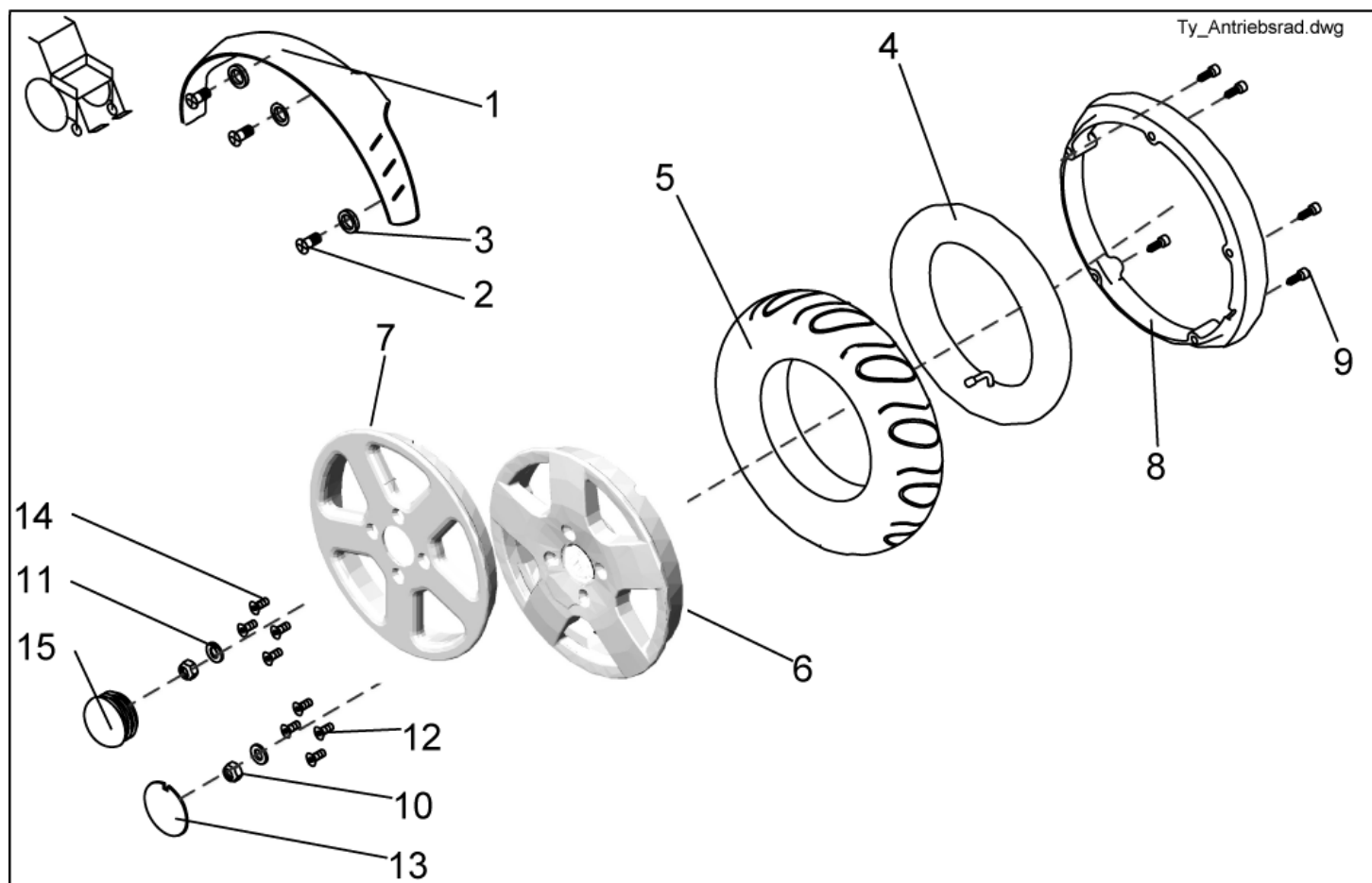Wheel Suspension/ Wheels - Driving Wheels detailed



item	ordercode	description	quantity
1	1434086-9007	SPLASH GUARD DRIVING WHEEL LEFT	1
1	1434085-9007	SPLASH GUARD DRIVING WHEEL RIGHT	1
2-3	1440567	MOUNTING KIT FOR SPLASH GUARDS DRIVING WHEELS	1
2	NA	BUTTONHEAD SOCKET SCREW, M6 X 20	6
3	NA	WASHER 6,4 X 12,5 X 1,6	6
4	M12016	INNER TUBE 3.00-8 WITH VALVE TR 87-90/90	1
5	1415173	COVER 3.00-8", GREY	1
5	M11968	TYRE 3.00X8, STOLLE, (TRELLEBORG), GREY	1
5	1430215	OUTER TYRE 3.00-8 GREY PUNCTURE-PROOF SOLID INLAY	1
6, 8+9	SP1423184	WHEEL RIM ASSEMBLY KIT 8", CPL.	2
7-9	1440637	MOUNTING KIT RIM THIN COMPLETE	2
6	NA	TYRE LOCKING RING 8", OUTSIDE	1
7	NA	WHEEL RIM 1,5 J X 8; OUTER	1
8	NA	TYRE LOCKING RING 8", INSIDE	1
8	NA	WHEEL RIM 1,5 J X 8; INNER	1
9	NA	SOCKET HEAD CAP SCREW M6 X 40	5
-	SP1423185	SPEEDOMETER DRIVE GEAR ASSEMBLY KIT, CPL.	1
10-13	SP1423443	WHEEL ASSEMBLY KIT 3.00-8	2
10+11,14+15	1446546	WHEEL ASSEMBLY KIT 3.00-8, THIN RIM	2
10	NA	NYLOC NUT M12 X 1,5	1
11	NA	WASHER 13 X 24 X 2,5	1
12	NA	SCREW M8 X 35 TORX DIN 7991	4
13	NA	CAP WITH "INVACARE" LOGO	1
14	NA	SCREW DIN7991 M8X20 TORX	4
15	NA	HUB CAP, GREY	1

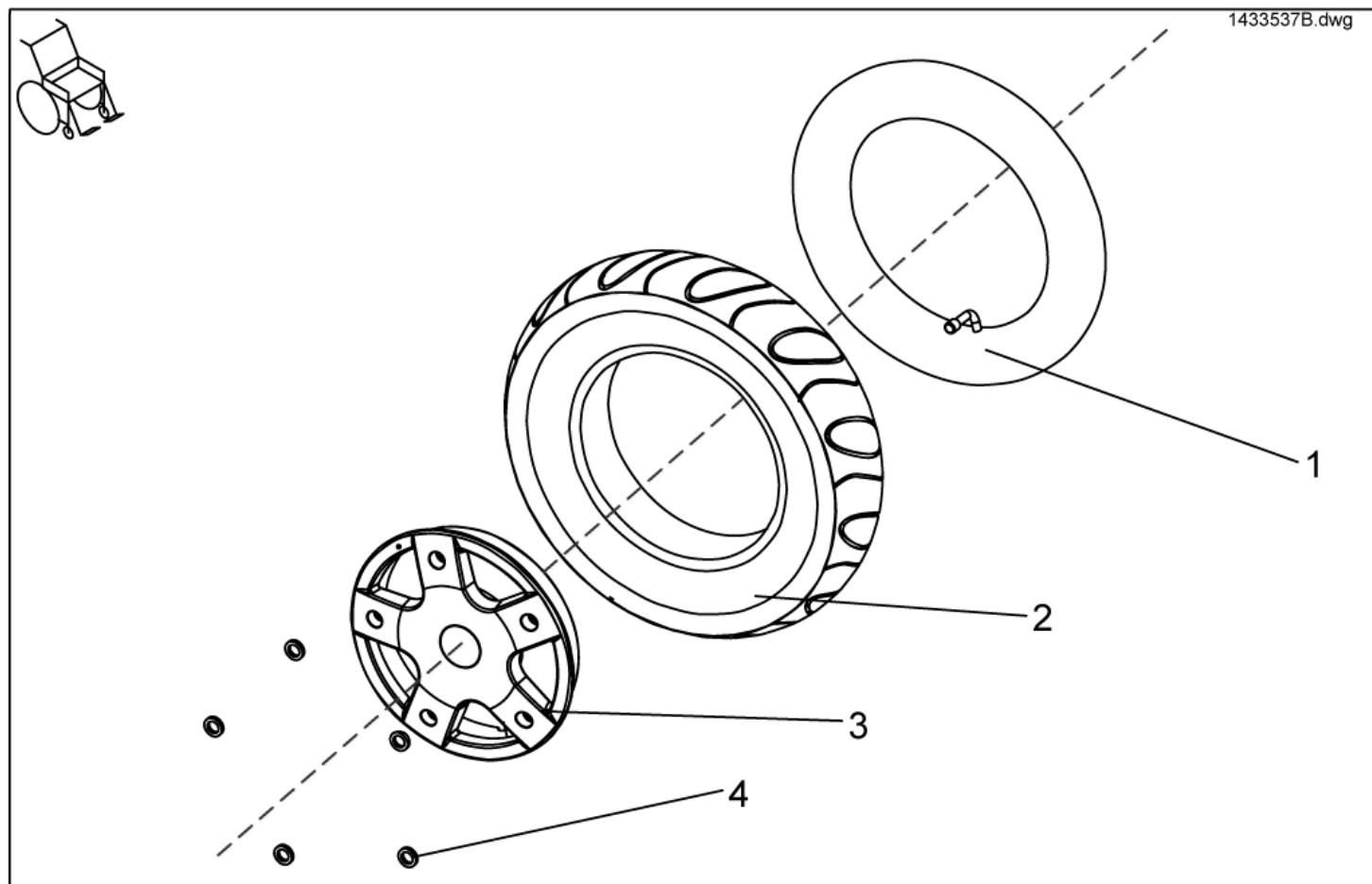
Wheel Suspension/ Wheels - Driving Wheels True Track



item	ordercode	description	quantity
1	M12016	INNER TUBE 3.00-8 WITH VALVE TR 87-90/90	1
2	1415173	COVER 3.00-8", GREY	1
2	M11968	TYRE 3.00X8, STOLLE, (TRELLEBORG), GREY	1
2	1430215	OUTER TYRE 3.00-8 GREY PUNCTURE-PROOF SOLID INLAY	1
3	1429347-0307	REAR WHEEL RIM OUTER	2
4	5829252	WASHER NORD-LOCK 5/16	5
-	5723604	SCREW DIN 912 M8x25 8.8 vz	5

Wheel Suspension/ Wheels - Crash kit

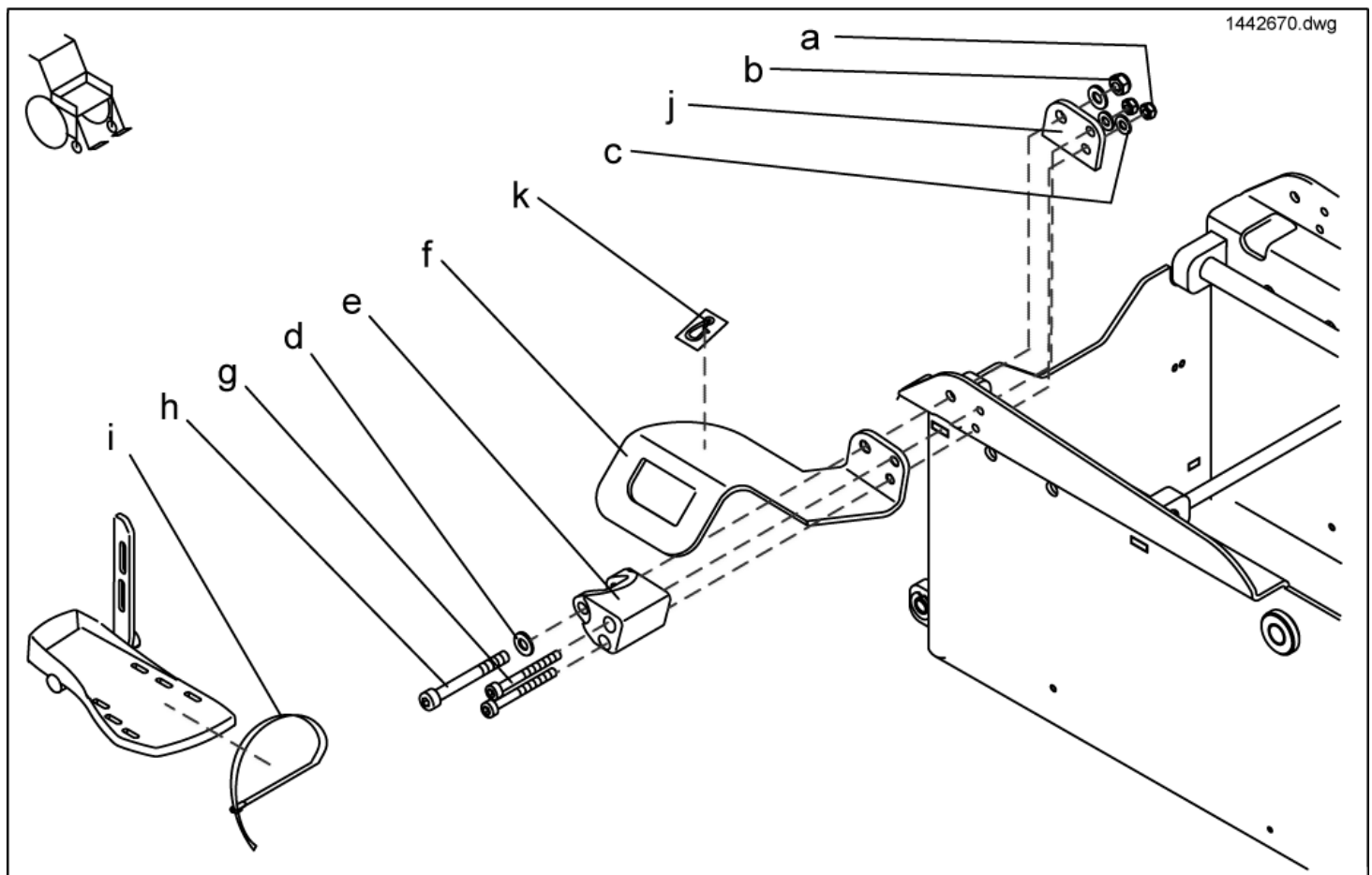
- ⚠ **CAUTION: Danger of injury if the wheelchair is not secured properly in the vehicle!**
In order to secure the empty wheelchair in the vehicle, please use the 4-Point-Tie-Down system. Please observe the chapter in the operating instructions on "Transport" for safe handling using the Standard transport system.
- ⚠ **CAUTION: Danger of injury if the wheelchair is not secured properly in the vehicle and if the user is not secured properly in the wheelchair.**
If a person is to be transported whilst sitting in the wheelchair you will require the 4-POINT-FIXATION (according to ISO 7176-19:2001) as well as a retaining belt (»»» see chapter on "retaining belts") and a headrest (»»» see chapter on "headrests"). The 4-POINT-FIXATION (according to ISO 7176-19:2001) has passed the test according to ISO 7176-19:2001 in connection with an anchoring system according to ISO 10542, retaining belt and headrest. It can, therefore, only be used in this configuration if the person remains seated in the wheelchair during transport. Please observe the chapter in the operating instructions on "transport" for safe handling using the 4-POINT-FIXATION (according to ISO 7176-19:2001).



item	ordercode	description	quantity
a-j	SP1442670	4-POINT-TIE-DOWN, WITH SHOCK ABSORBERS RED, COMPLETE	1
a-j	SP1442672	4-POINT-TIE-DOWN, WITH SHOCK ABSORBERS BLUE, COMPLETE	1
a-j	SP1442673	4-POINT-TIE-DOWN, WITH SHOCK ABSORBERS GREEN, COMPLETE	1
a-k	1442765	4-POINT-FIXATION (according to ISO 7176-19:2001) WITH SHOCK ABSORBERS RED, COMPLETE, Requires additionally H-Belt and headrest compulsory!	1
a-k	1442764	4-POINT-FIXATION (according to ISO 7176-19:2001) WITH SHOCK ABSORBERS BLUE, COMPLETE, Requires additionally H-Belt and headrest compulsory!	1
a-k	1442763	4-POINT-FIXATION (according to ISO 7176-19:2001) WITH SHOCK ABSORBERS GREEN, COMPLETE, Requires additionally H-Belt and headrest compulsory!	1
a	NA	NYLOC NUT M6	4
b	NA	NYLOC NUT M8	2

item	ordercode	description	quantity
c	NA	WASHER 6,4 X 12,5 X 1,6	4
d	NA	WASHER 8,4 X 17 X 1,6	4
e	NA	SHOCK ABSORBER CRASH KIT, MOUNT LEFT	1
e	NA	SHOCK ABSORBER CRASH KIT, MOUNT RIGHT	1
f	1440375-9005	TIE STRAP, LEFT-HAND	1
f	1440376-9005	TIE STRAP, RIGHT-HAND	1
g	NA	SCREW M6X75 DIN 912	4
h	NA	SOCKET HEAD CAP SCREW M8 X 70	2
i	F384020	CARRYING STRAP	2
j	1442565-0003	BACKING SHEET	2
k	NA	ISO 7176-19 STICKER RAL 1026	4

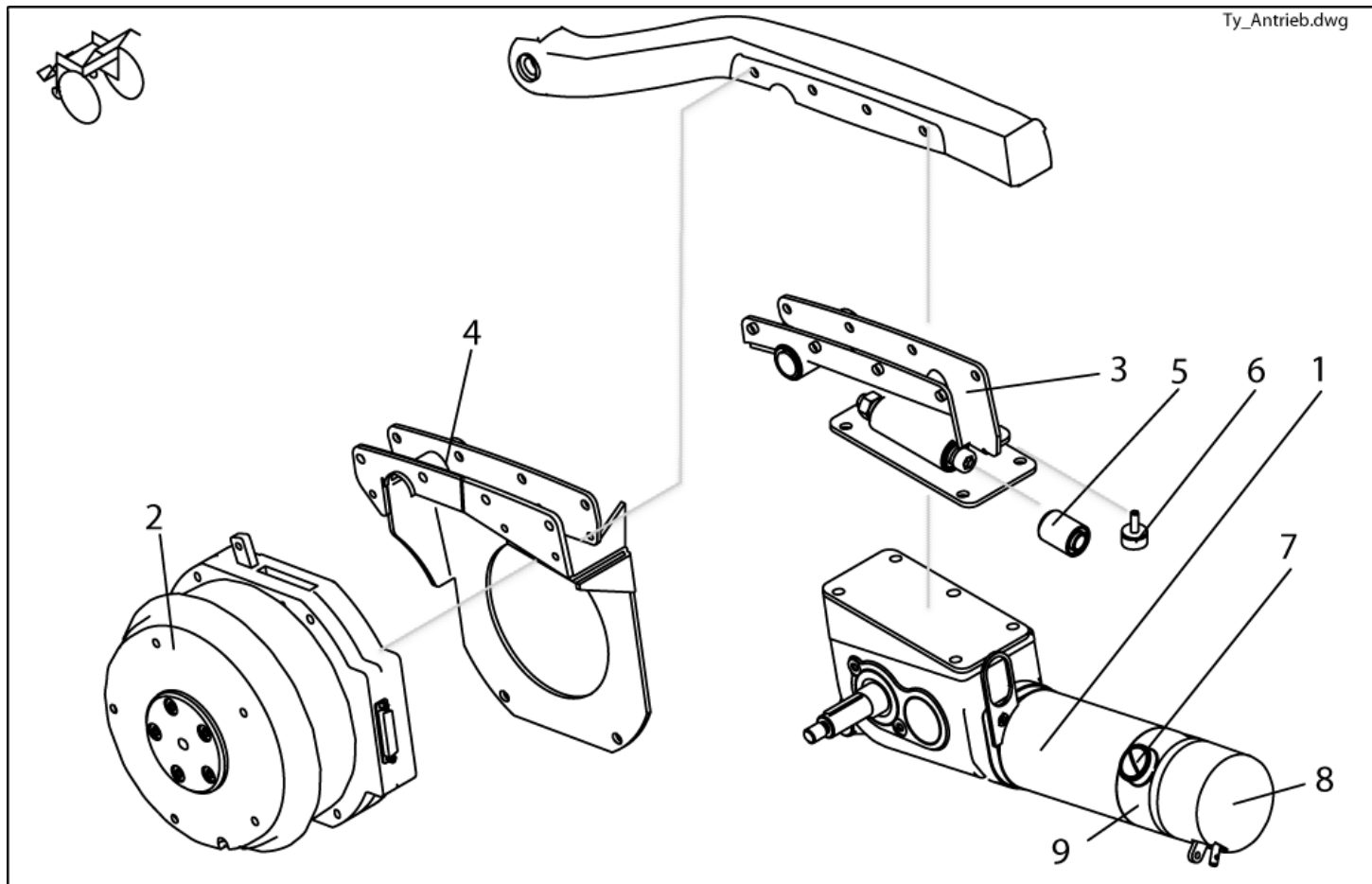
Driving unit



item	ordercode	description	quantity
1	1440363	DRIVE UNIT 6KM/H LEFT	1
1	1440364	DRIVE UNIT 6KM/H RIGHT	1
1	1440361	DRIVE UNIT 10KM/H LEFT	1
1	1440362	DRIVE UNIT 10KM/H RIGHT	1
2	SP1433563	MOTOR GEARLESS LH	1
2	SP1433564	MOTOR GEARLESS RH	1
3	1440673	HANGING SYSTEM DRIVING UNIT LEFT COMPLETE	1
3	1440674	HANGING SYSTEM DRIVING UNIT RIGHT COMPLETE	1
4	1440569	CRANK TRUE TRACK LEFT COMPLETE	1
4	1440570	CRANK TRUE TRACK RIGHT COMPLETE	1
5	N02190	ROCKER-ARM BUSH	4
6	1440366	RUBBER METAL STOPPER	4
-	1433044	WHEEL HUB STD.	1
-	1415171	WHEEL HUB FOR BRAKE	1
7	P09293	CARBON BRUSHES SN: ==> 220001	4
7	1488052	CARBON BRUSHES SN: 220001==>	4
8	1510060	COVER FOR MAGNETBRAKE	1
9	1510059	MAGNETBRAKE	1

Brake

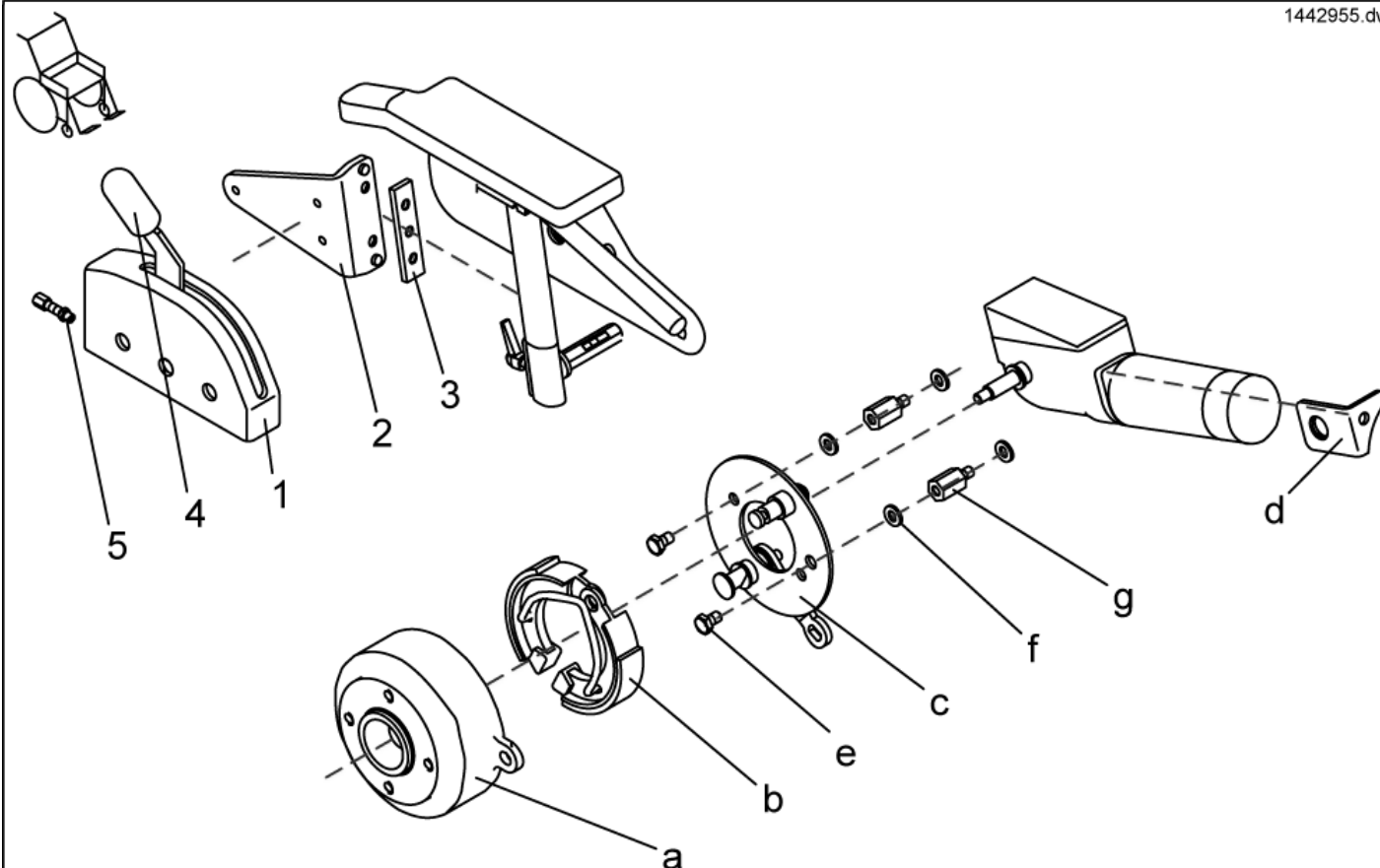
1442955.dwg




item	ordercode	description	quantity
-	SPF331028	BRAKE ASSY LH	1
-	SPF331038	BRAKE ASSY RH	1
-	SP1423473	BOWDEN CABLE SET CPL.	1
1	F331022	BRAKE BOX COVER; GREY	1
2	1428075-9005	BRAKE BOX HOLDER LH	1
2	1428076-9005	BRAKE BOX HOLDER RH	1
3	1421720-0002	GROVE PLATE	1
4	5007791	KNOB; BLACK	1
5	M30050	ADJUSTING SCREW M6 X 24; ZINC PLATED	4
a,b,c,d,e,f,g	1442955	Kit WHEEL BRAKE LH, CPL.	1
a,b,c,d,e,f,g	1442956	Kit WHEEL BRAKE RH, CPL.	1
a	1415171	WHEEL HUB FOR BRAKE	1
b	M30985	BRAKE SHOE 90X18 (ANELLI)	1
c	SP1415066	PRE-ASSEMBLED BRAKE BACKING PLATE RH	1
c	SP1415067	PRE-ASSEMBLED BRAKE BACKING PLATE LH	1
d	1421653-0003	HOLDER-UP LH	1
d	1421654-0003	HOLDER-UP RH	1
e	NA	HEX HEAD SCREW M6 X 10	2
f	NA	WASHER 6,4 X 12,5 X 1,6	4
g	NA	HEXAGON SPACER	2

Decoupling

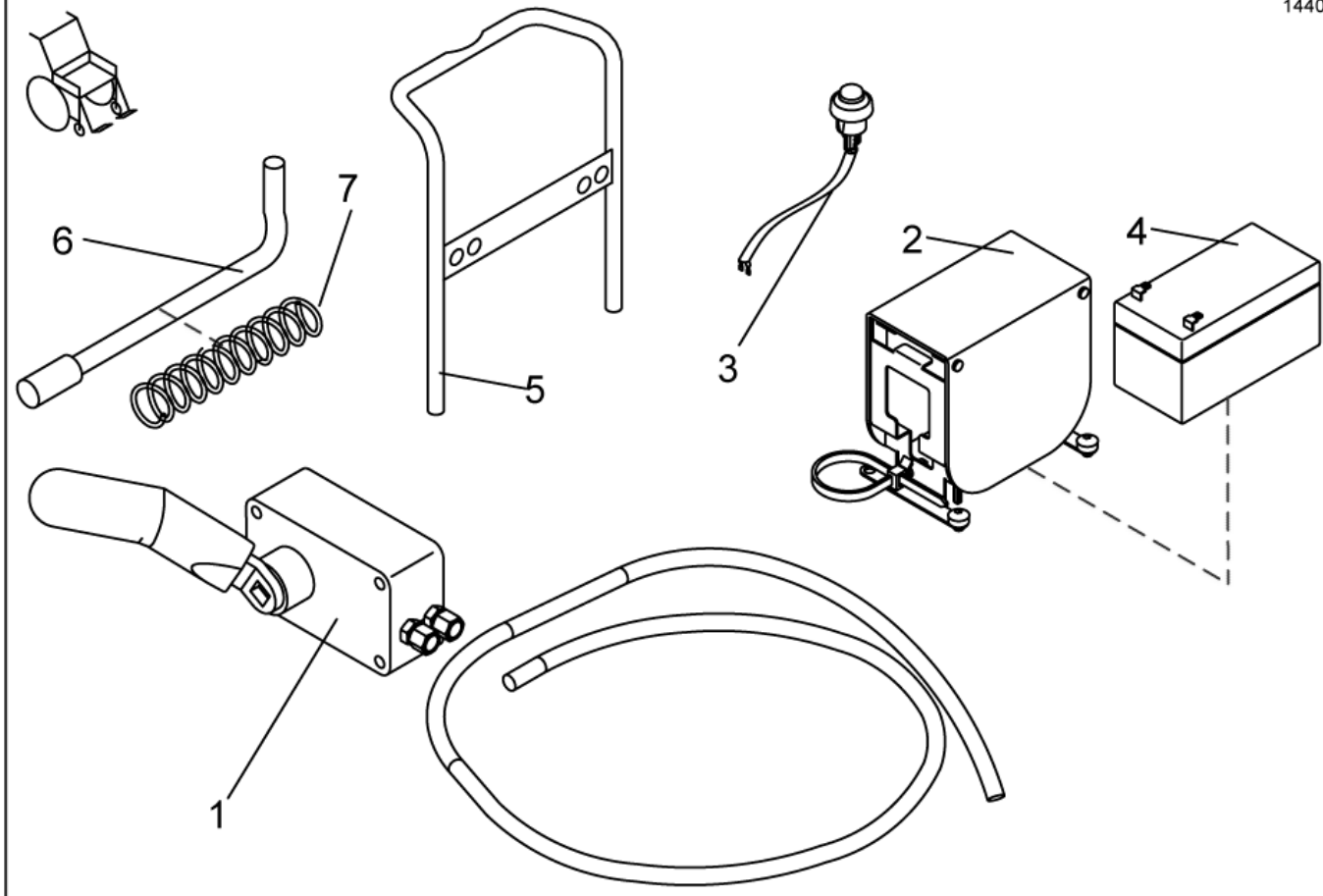1440343.dwg




item	ordercode	description	quantity
1	SP1440343	DECOUPLING COMPLETE	1
2-4	SP1440344	ELECTRONICAL COUPLING FOR TRUE TRACK COMPLETE	1
3	1440729	MOMENTARY CONTACT FOR EL. DECOUPLING TRUE TRACK WITH CABLE	1
4	1440386	BATTERY FOR ELECTRICAL DECOUPLING (12V, 1,2AH, FIAMM-GS)	1
5	1441556-9005	HANDLE FOR ARNE JENSEN SEAT WITH ELECTRICAL DECLUTCHING TRUE TRACK	1
6+7	1440568	EMERGENCY DECOUPLING TRUE TRACK COMPLETE	2
7	1441591	PRESSURE SPRING	1

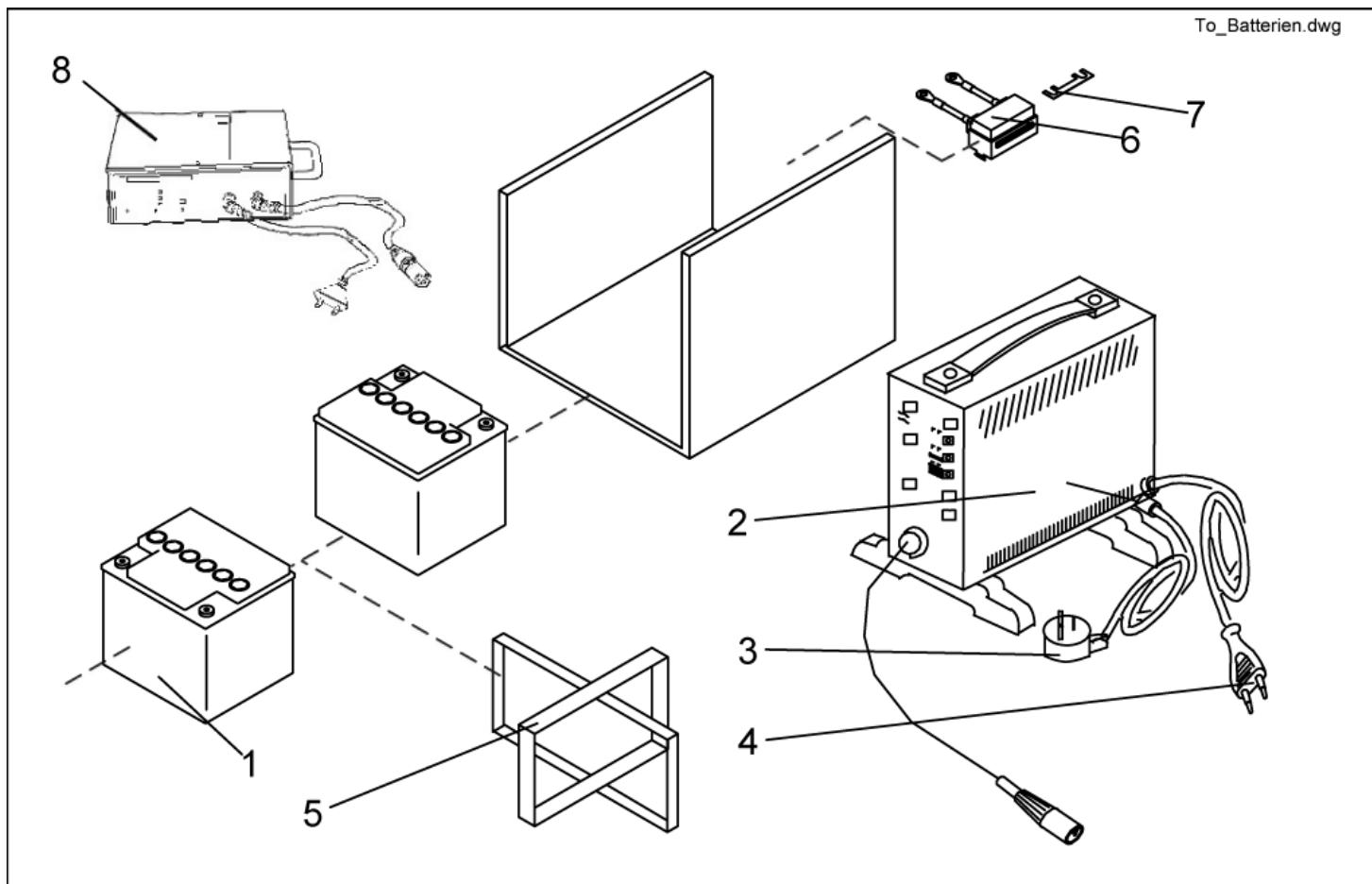

item	ordercode	description	quantity
1	1432944	BATTERY MK SLDG 60AH	2
2	SP1423696	CHARGER 24V/8A W. CONTINENTAL MAINS CABLE, CPL.	1
2	SP1423697	CHARGER 24V/8A W. UK MAINS CABLE, CPL.	1
2	P51981	BATTERY CHARGER 8 A WITH XLR CONNECTION	1
3	P21080	MAINS CABLE GCB-0403 UK; BLACK	1
4	P21079	MAINS CABLE GCB-0401 CONTINENTAL; BLACK	1
5	1440403	BATTERY STRAP	2
6	1436210	FUSE BRACKET WITH FUSE 50A	1
6	1436212	FUSE BRACKET WITH FUSE 80A (FOR TRUE TRACK ONLY!)	1
7	P12191	FUSE 50 A	1
7	1433734	STRIP FUSE 80A (ONLY TRUE TRACK!)	1
-	1436209	ACS-BATTERY CABLE 60AH STANDARD	1
-	1436211	BATTERY CABLE FOR TRUE TRACK 60AH	1
8	5813987	Ladegerät 8a Gi Popular	1

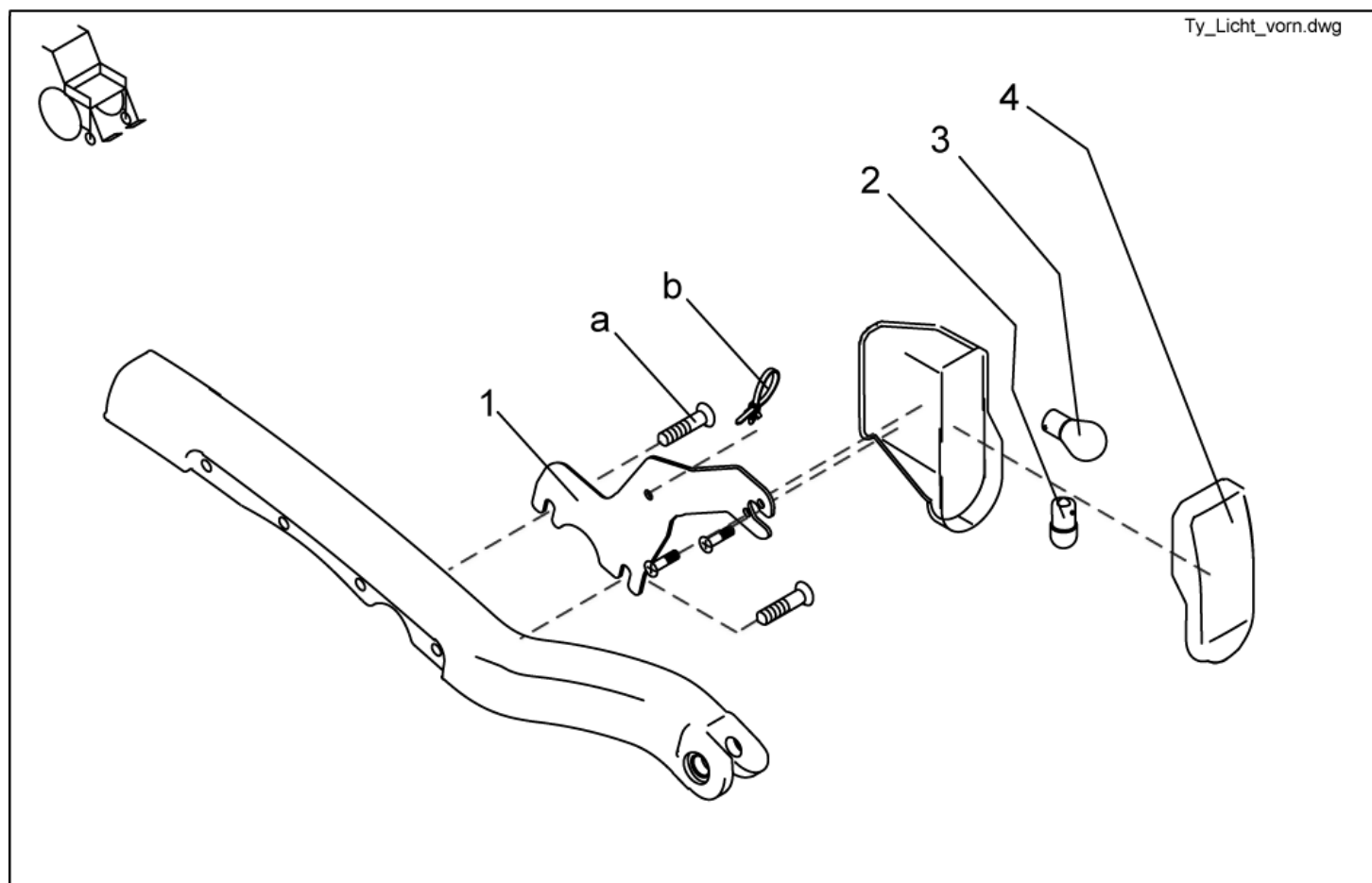
Lighting - Front Lights




item	ordercode	description	quantity
-	1440426	HEADLAMP LEFT WITH CABLE	1
-	1442734	HEADLAMP RIGHT WITH CABLE	1
1	1436367-9005	HOLDER HEADLAMP LEFT	1
1	1436368-9005	HOLDER HEADLAMP RIGHT	1
2	P31108	LIGHTBULB 24V/10W	2
3	1419271	LIGHTBULB 24V/21W, YELLOW	2
4	1433737	HEADLIGHT LENS	2
a-b	1440740	MOUNTING KIT FOR LAMP HOLDER AND HEADLIGHT FOR TYPHOON	1
a	NA	BOLT 4,8 X 25 DIN 7981	4
b	NA	CABLE BINDER 5,0x225 SCHWARZ	2

Lighting - Rear Lights

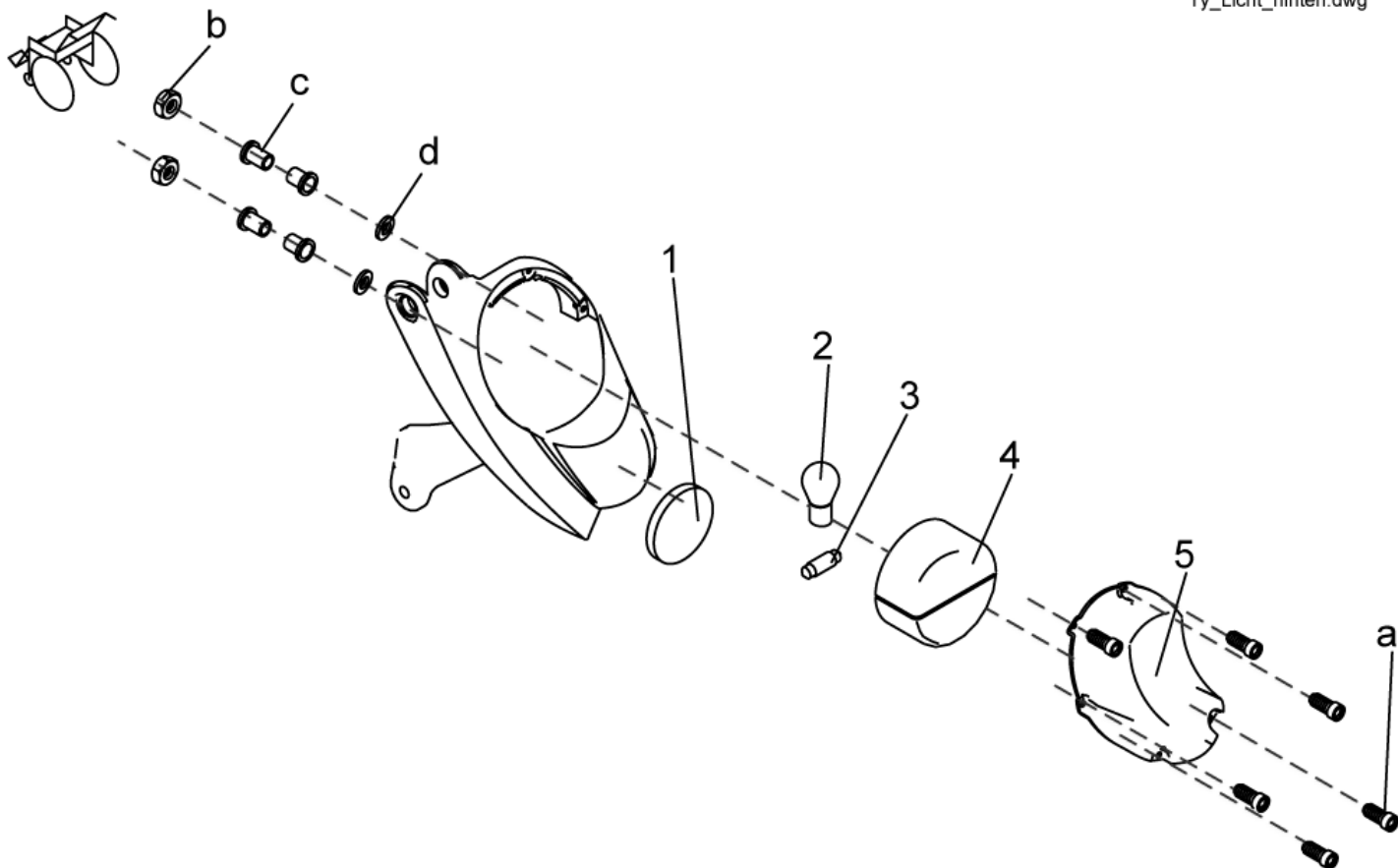Ty_Licht_hinten.dwg



item	ordercode	description	quantity
1	1419274	REFLECTOR, RED	2
2	1419271	LIGHTBULB 24V/21W, YELLOW	2
3	P31191	LAMP 24V/5W	2
4	1440427	REAR LIGHT WITH CABLE RIGHT	1
5	1434101	REAR LIGHT COVER LEFT	1
5	1434097	REAR LIGHT COVER RIGHT	1
a-d	1440563	MOUNTING KIT REAR LIGHT	2
a	NA	M4X8 SCREW	6
b	NA	NYLOC NUT M6	2
c	NA	INSULATING SHEATH 4MM	4
d	5829246	WASHER 6X12X1,6 Nylon D, 125A	4

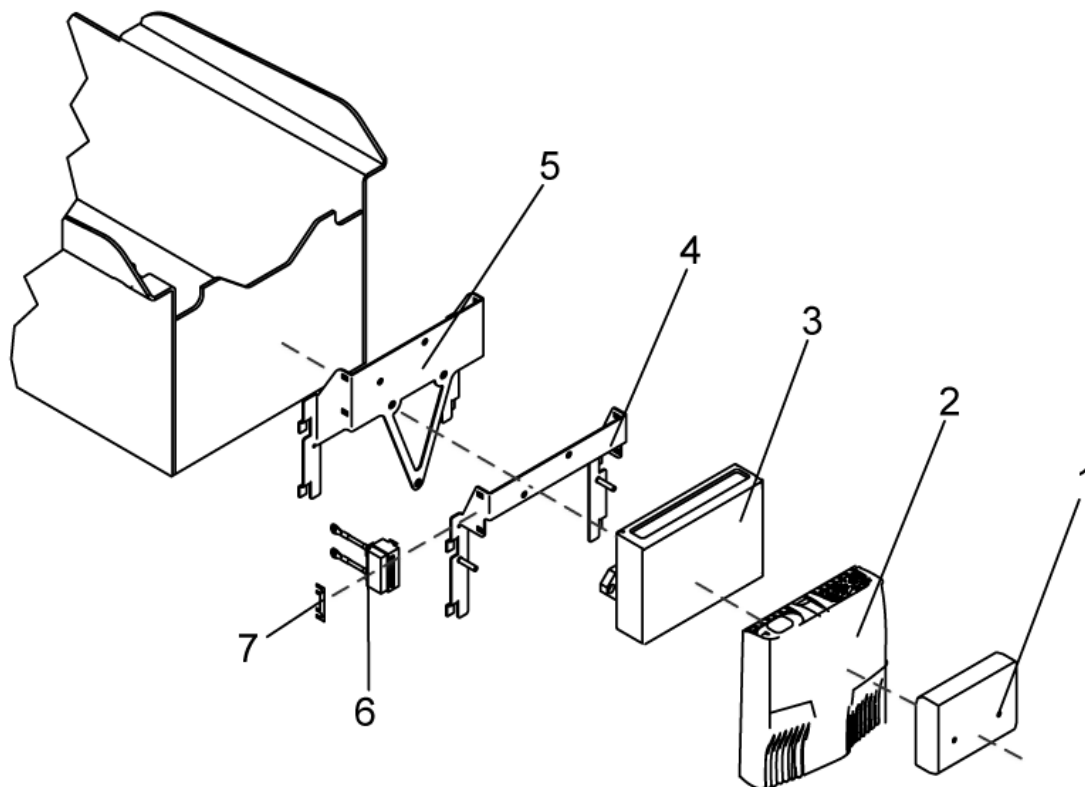

item	ordercode	description	quantity
1	6001825	ACS-LIGHTING MODULE 24 V STD.	1
1	P10081	ACS-CLAM (LIGHTING/ACTUATOR MODULE COMBINED)	1
2	1430150	ACS POWER MODUL DX-GB 100A	1
2	AT.1430150	Austauschteil für ACS Powermodul True Track DX-GB 100A	1
3	P10064	ACS POWER MODULE DX-PM2 (80A)	1
4	1440371-0002	HOLDER ACS-ELECTRONIC	1
5	1440372-0002	HOLDER POWERMODULE TRUE TRACK	1
6	1436210	FUSE BRACKET WITH FUSE 50A	1
6	1436212	FUSE BRACKET WITH FUSE 80A (FOR TRUE TRACK ONLY!)	1
7	P12191	FUSE 50 A	1
7	1432561	STRIP FUSE 80A (ONLY TRUE TRACK!)	1
-	1415004	CENTRAL LIGHT DISTRIBUTOR MAINBOARD	1
-	P10072	ACS-BUS T-ADAPTER (SPLITTER)	1
-	1440419	MAIN CABLE HARNESS ACS WITH LIGHT AND ALL ACTUATORS	1
-	1440434	MAIN CABLE HARNESS WITH LIGHTS WITHOUT LIFTER	1
-	1440429	ACS CABLE HARNESS, LIGHTS ONLY	1

Cables

battery cables

ordercode	ACS	ACS True Track
1436211		X
1436209	X	

fuse cables

ordercode	ACS	ACS True Track
1436212		X
1436210	X	


ACS-Buscables

ordercode	ACS without Lights	ACS Lights only	ACS with Lights without Lifter	ACS with Lights with all Actuators
1440424	X	X		X
GSM63016	X	X	X	
GSM63003		X	X	X


Main cable harnesses

ordercode	ACS Lights only	ACS with Lights without Lifter	ACS with Lights with all Actuators
1440429	X		
1440434		X	
1440419			X

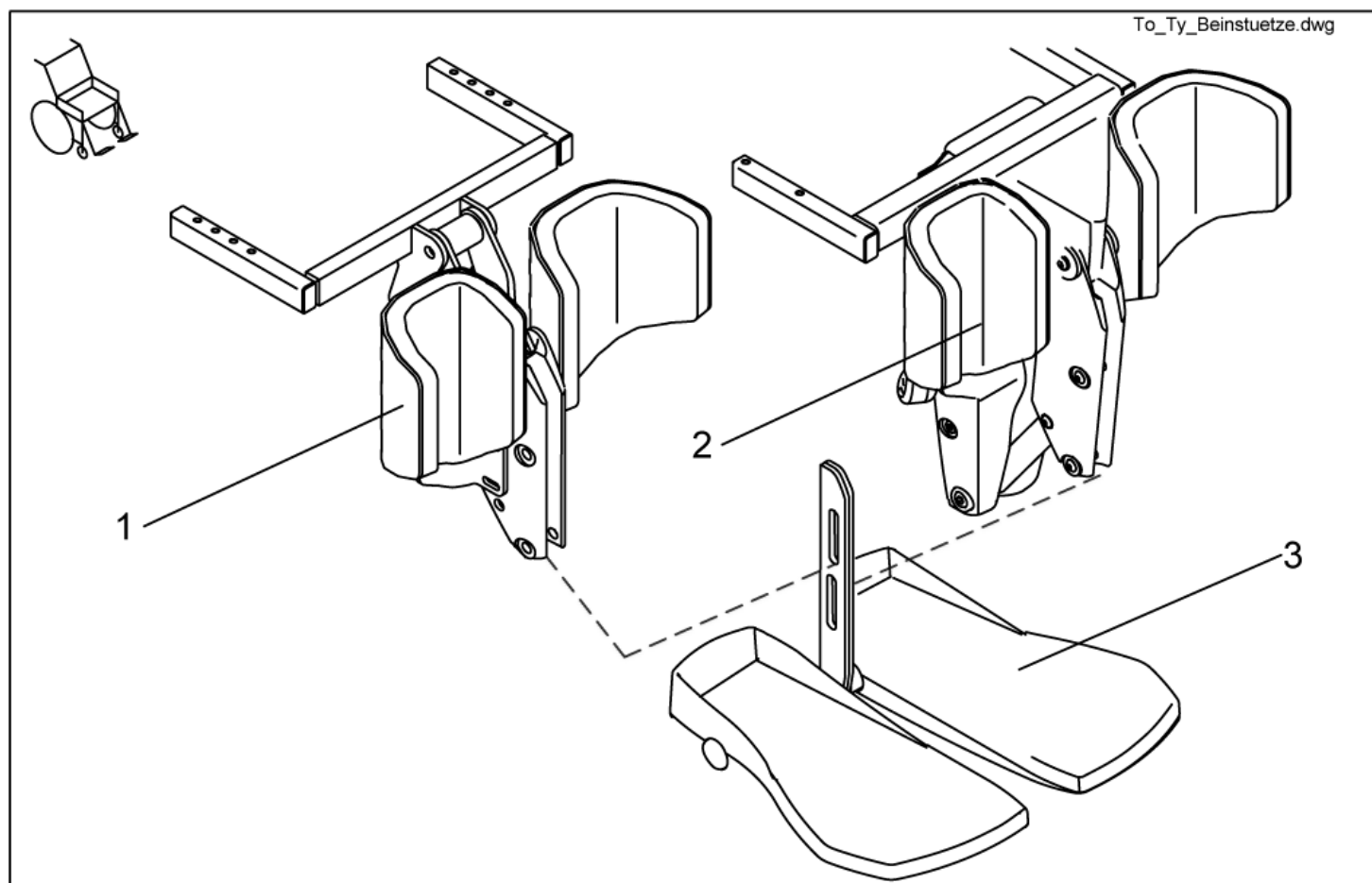Legrests, central mounted




item	ordercode	description	quantity
1	SP1436512	CENTRAL HANGER, MANUAL, COMPLETE	1
2	SP1436513	CENTRAL HANGER 100°, ELECTRICAL COMPLETE	1
2	SP1440413	CENTRAL HANGER 90°, ELECTRICAL, COMPLETE	1
3	SP1440290	CENTRAL LOWER LEGREST, SMALL (32-38cm), COMPLETE	1
3	SP1440291	CENTRAL LOWER LEGREST, MEDIUM (38-44cm), COMPLETE	1
3	SP1440292	CENTRAL LOWER LEGREST, LARGE (44-50cm), COMPLETE	1

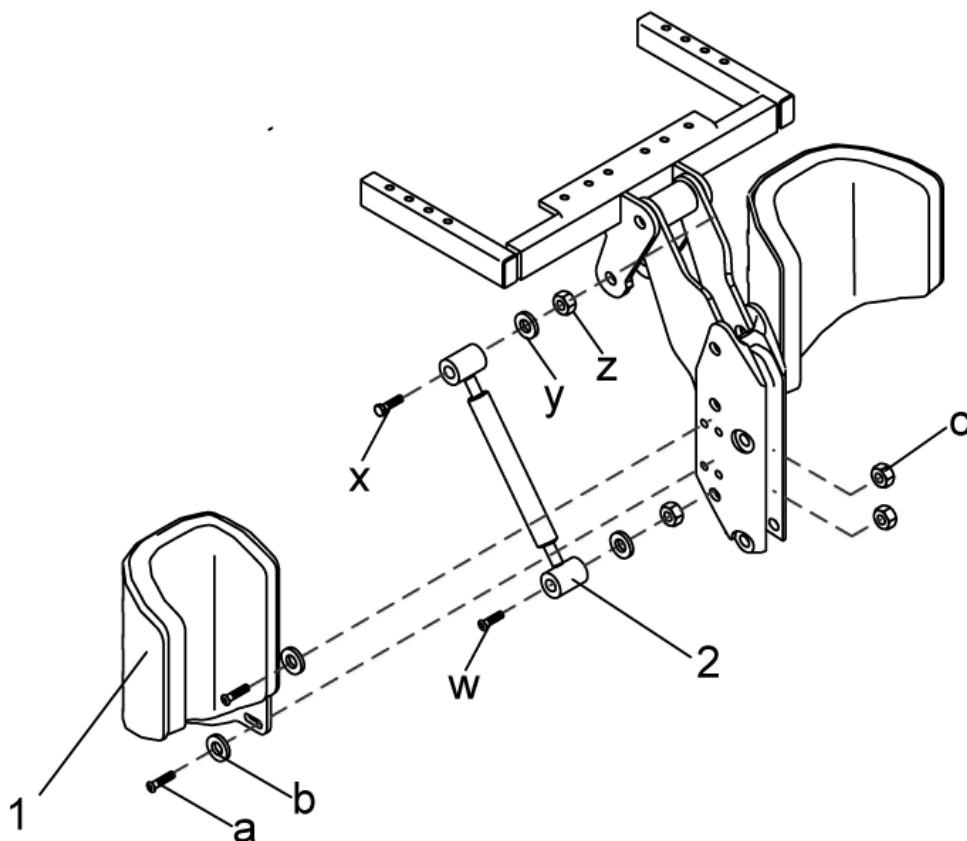

item	ordercode	description	quantity
1	1436069	CALF PLATE LEFT, COMPLETE WITH HOLDER	1
1	1436070	CALF PLATE RIGHT, COMPLETE WITH HOLDER	1
a-c	1440615	MOUNTING KIT CALF SUPPORT (MANUAL VERSION)	2
a	NA	M6X16 ISO7380 SCREW	2
b	NA	WASHER 6,4 X 12,5 X 1,6	2
c	NA	NYLOC NUT M6	2
2	1436226	SPINDLE	1
w-z	1440617	MOUNTING KIT SPINDLE	1
w	NA	M10X50 ISO7380 SCREW	1
x	NA	M10X55 DIN 931 SCREW	1
y	NA	WASHER 10,5 X 21 X 2 DIN 125	2
z	NA	NYLOC NUT M10 DIN 985	2

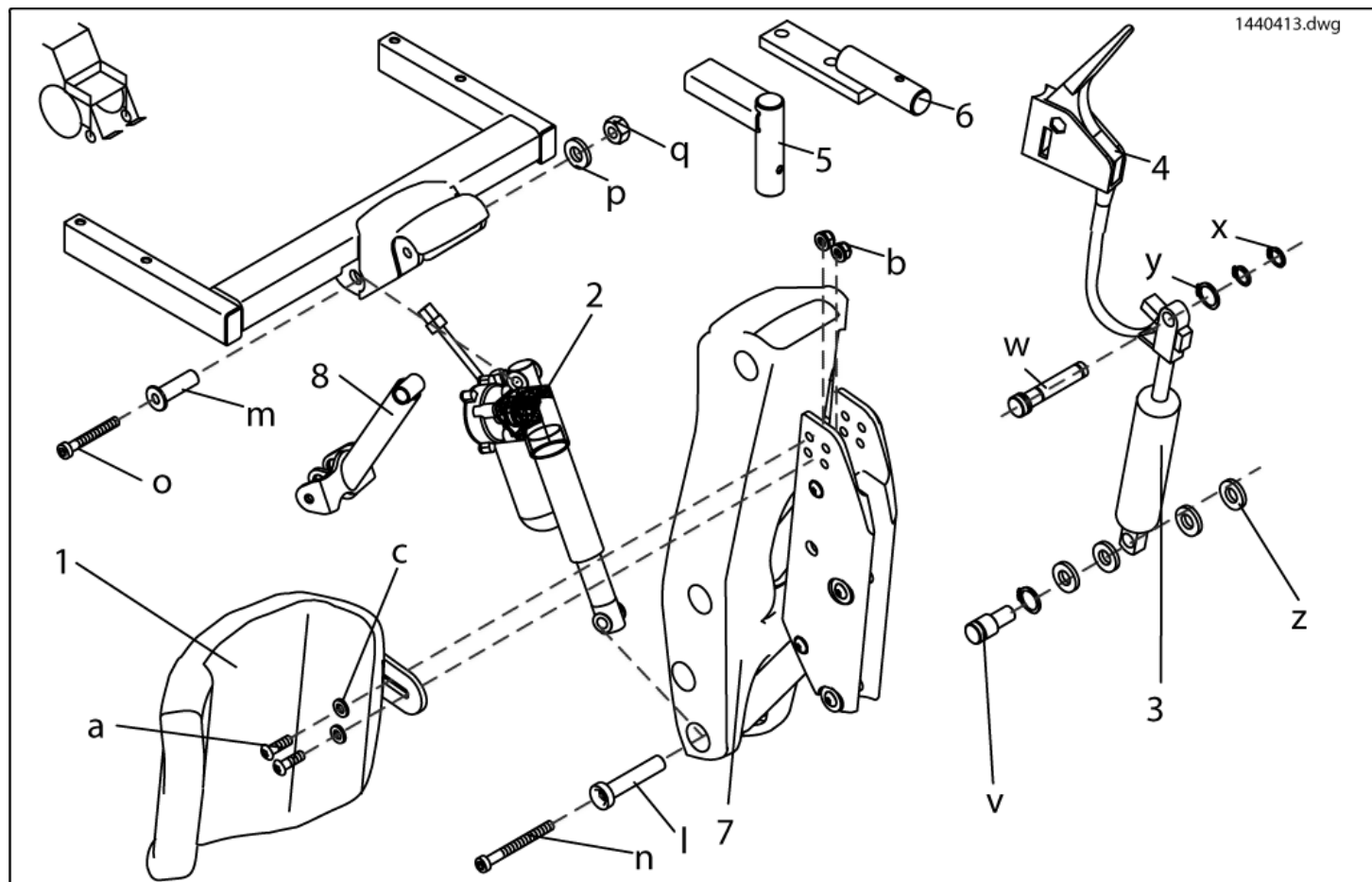
Hanger electrical




item	ordercode	description	quantity
1	1436076	CALF PLATE WITH HOLDER LEFT	1
1	1440594	CALF PLATE WITH HOLDER RIGHT	1
a-c	1440620	MOUNTING KIT FOR CALF PLATE (ELECTRICAL VERSION)	2
a	NA	BUTTONHEAD SOCKET SCREW, M6 X 20	4
b	NA	NYLOC NUT M6	4
c	NA	WASHER 6,4 X 12,5 X 1,6	4
2	1440600	ACTUATOR ELECTRICAL LEGREST 90°	1
2	1440592	ACTUATOR ELECTRICAL LEGREST 100°	1
l-q	1440621	MOUNTING KIT ACTUATOR	1
l	NA	BUSHING REAC LONG	1
m	NA	BUSHING REAC	1
n	NA	M6X60 DIN6912 SCREW ELECTRO-GALVANISED	1
o	NA	M6X45 DIN6912 8.8 CHEESE HEAD SCREW WITH ALLEN KEY ELECTRO-GALVANISED	1
p	NA	NYLOC NUT M6	1
q	NA	WASHER 6,4 X 18 X 1,6	1
3+4	SP1440593	GAS PRESSURE SPRING COMPLETE	1
4	1441590	ASSEMBLY LEVER, FRAME	1
v-z	1440622	MOUNTING KIT GAS PRESSURE SPRING	1
v	NA	BOLT	1
w	NA	BOLT	1
x	NA	RETAINER RING 10 MM	2
y	NA	CAP DIN 471 14 X 1	2
z	NA	WASHER 12.2 X 18.0 X 1.0	4
5	1440603-9005	SUPPORT FOR RELEASE LEVER	1
6	1442727-9005	SUPPORT FOR RELEASE LEVER (ARNE JENSEN ONLY)	1
7	1434389-9006	HOUSING	1
8	SP1436082	ARM 3 WITH BUSHES	1

Lower Legrests

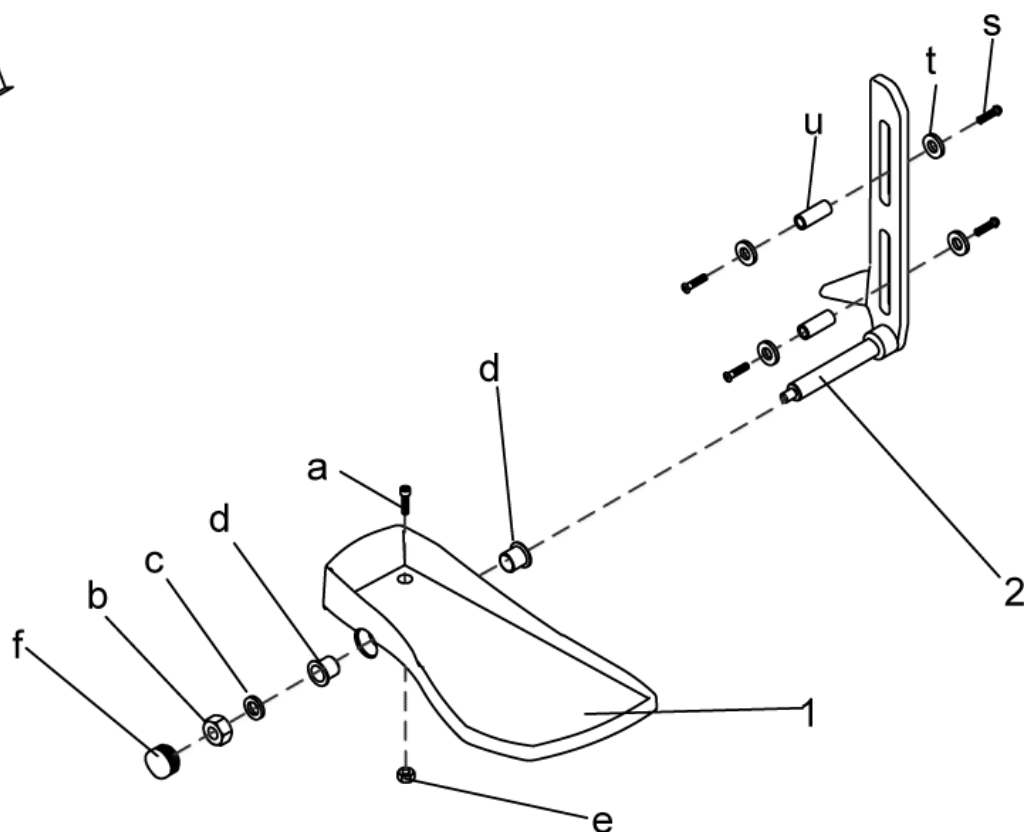1440290.dwg



item	ordercode	description	quantity
1	1440613	FOOTPLATE WITH RUBBER AND BUSHES RIGHT	1
1	1440612	FOOTPLATE WITH RUBBER AND BUSHES LEFT	1
a-f	1440614	MOUNTING KIT FOOTPLATE	2
a	NA	SOCKET HEAD CAP SCREW M6 X 25	1
b	NA	NYLOC NUT M8	1
c	NA	WASHER 8,4 X 17 X 1,6	1
d	NA	BUSH BB 1517 DU	2
e	NA	HEX NUT M6	1
f	NA	CAP	1
2	1440442-0003	SHORT (32-38cm), LEFT-HAND LINEAR COMPENSATION	1
2	1440445-0003	SHORT (32-38cm), RIGHT-HAND LINEAR COMPENSATION	1
2	1440443-0003	MEDIUM (38-44cm), LEFT-HAND LINEAR COMPENSATION	1
2	1440446-0003	MEDIUM (38-44cm), RIGHT-HAND LINEAR COMPENSATION	1
2	1440444-0003	LONG (44-50cm), LEFT-HAND LINEAR COMPENSATION	1
2	1440447-0003	LONG (44-50cm), RIGHT-HAND LINEAR COMPENSATION	1
s-t	1440616	MOUNTING KIT FOR LINEAR COMPENSATION	1
s	NA	M6X16 ISO7380 SCREW	4
t	NA	WASHER 6.4 DIN 9021	4
u	NA	SHAFT	2

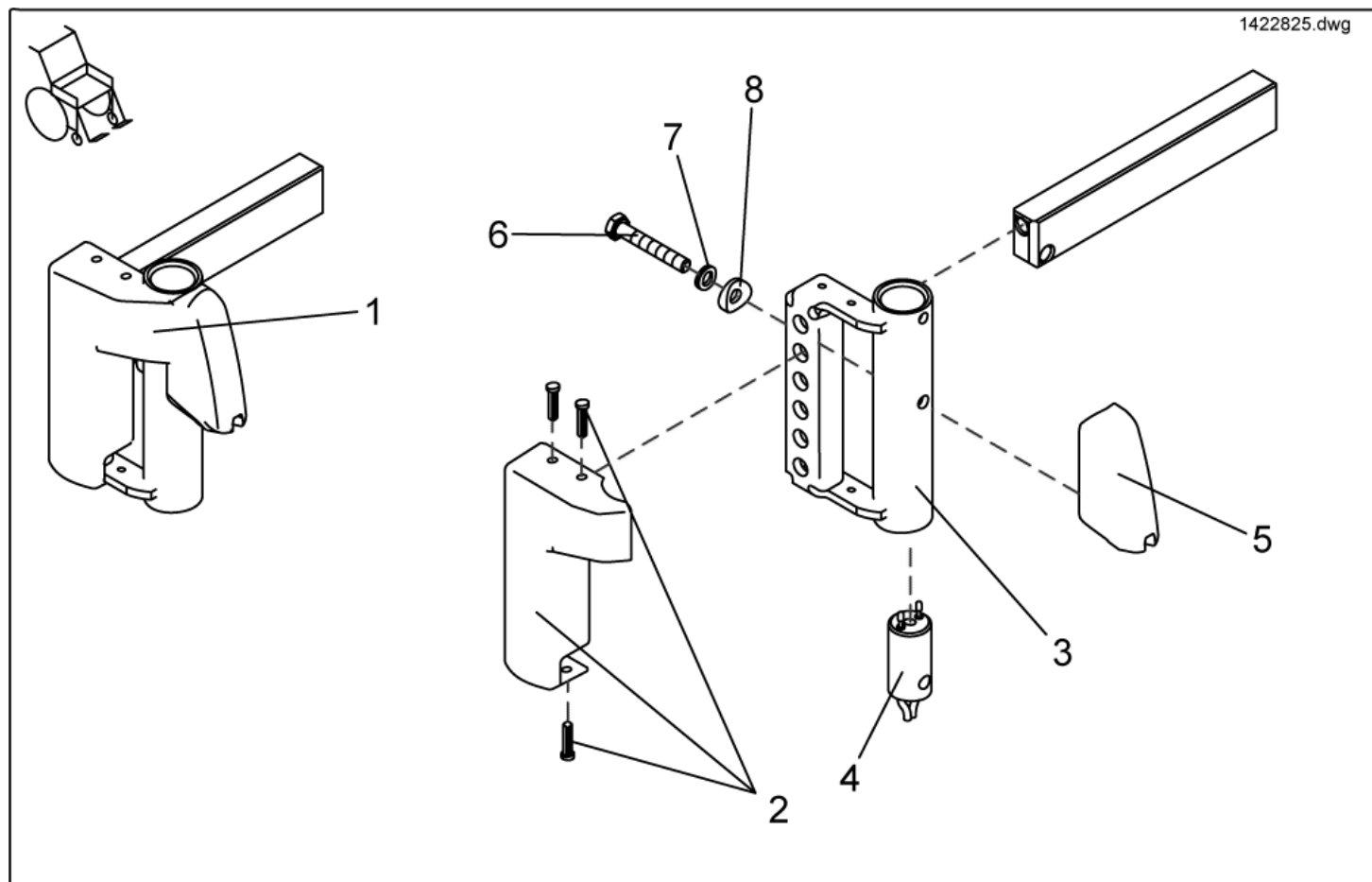


item	ordercode	description	quantity
1	SP1417917	LEG SUPPORT CARRIER LH MG HS SEAT FRAMES	1
1	SP1417918	LEG SUPPORT CARRIER RH MG HS SEAT FRAMES	1
1	SP1422825	FOOTREST SUPPORT LH F. ELECTRIC LEGREST	1
1	SP1422826	FOOTREST SUPPORT RH F. ELECTRIC LEGREST	1
1	SP1450407	FOOTREST SUPPORT RH F. ELECTRIC LEGREST	1
2	1434961	COVER LH WITH PINE TREE	1
2	1434960	COVER RH WITH PINE TREE	1
3	NA	FOOTREST RECEPTACLE, LEFT	1
3	NA	FOOTREST RECEPTACLE, RIGHT	1
4	1415097	LEGREST CONTACT W. CABLE 700 MM	2
5,6,7,8	SP1423192	LEGREST ADAPTER ASSEMBLY KIT	2
5	NA	FRAME ADAPTER (ALU)	1
6	NA	SCREW M6 X 40, DIN 933	1
7	NA	NORD-LOCK WASHER 6 MM	1
8	NA	SPACER	1



Note:

The Power supply cable, ordercode 1415097 is not compatible with Footrest receptacle starting from year of construction 07/2005.

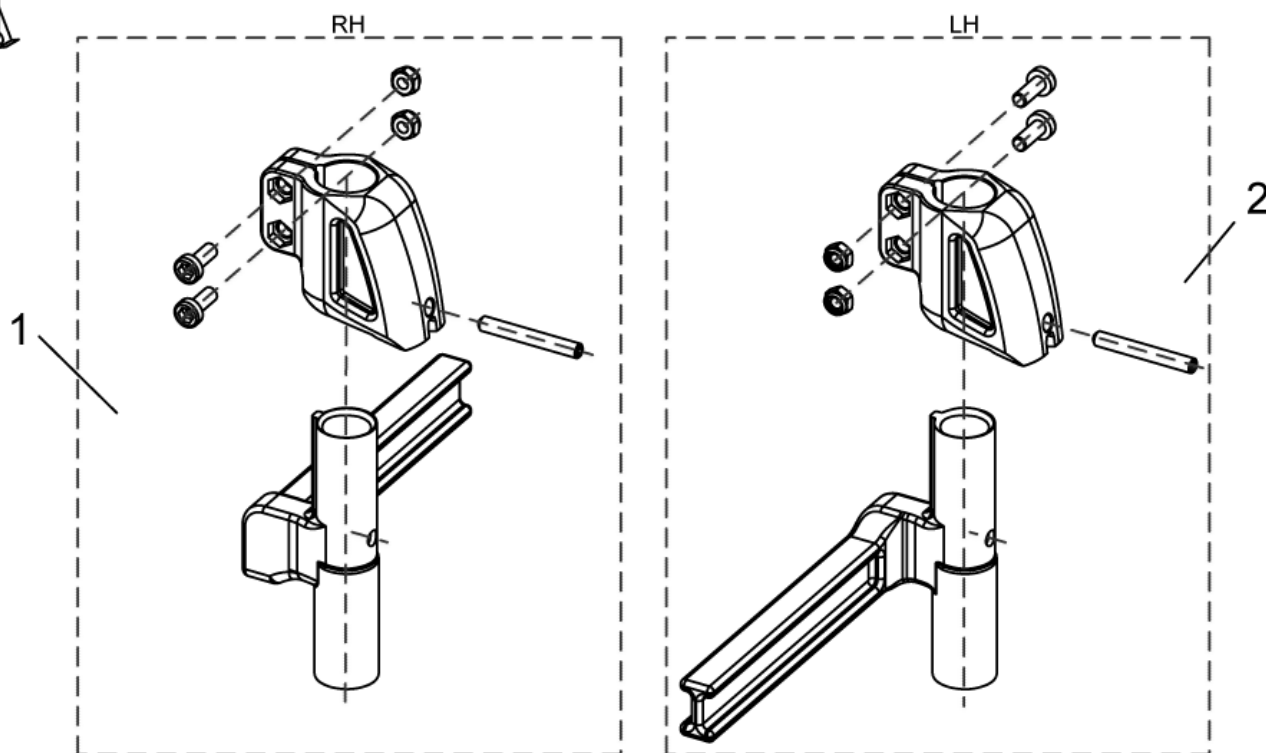1449732_1449733.dwg



item	ordercode	description	quantity
1	SP1449732	FOOTREST RECEPTACLE, COMPLETE, MANUAL RH 5CM	1
2	SP1449733	FOOTREST RECEPTACLE, COMPLETE, MANUAL LH 5CM	1

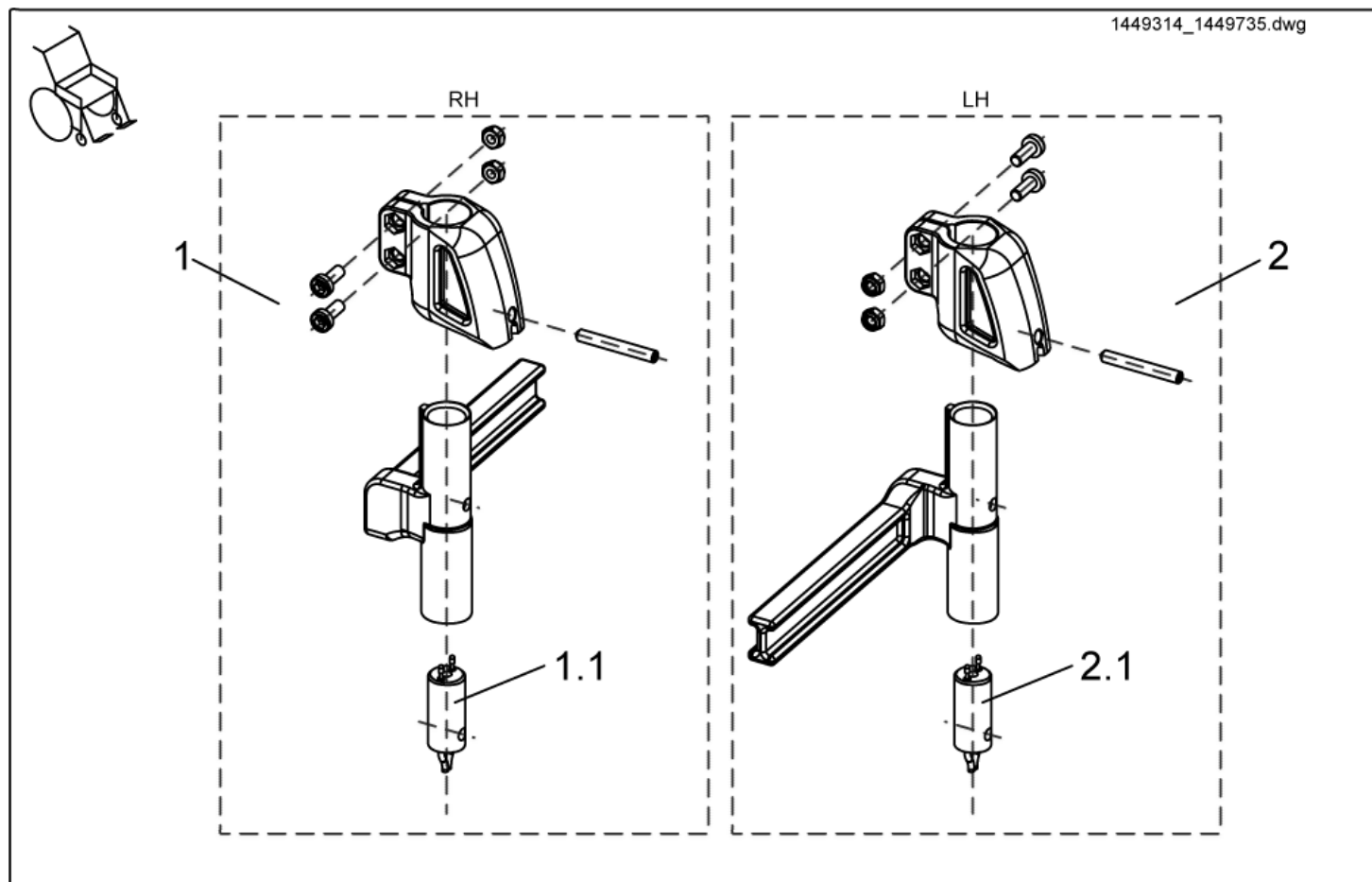


item	ordercode	description	quantity
1	SP1449734	FOOTREST RECEPTACLE, CPL., ELECTRIC RH 5CM	1
1.1	1450228	POWER SUPPLY CABLE FOOTREST, LONG (BLACK ADAPTER)	1
2	SP1449735	FOOTREST RECEPTACLE, CPL., ELECTRIC LH 5CM	1
2.1	1450228	POWER SUPPLY CABLE FOOTREST, LONG (BLACK ADAPTER)	1



Note:


The Power supply cable, ordercode 1450228 and 1450232 are not compatible with Footrest receptacle starting from year of construction 07/2005.

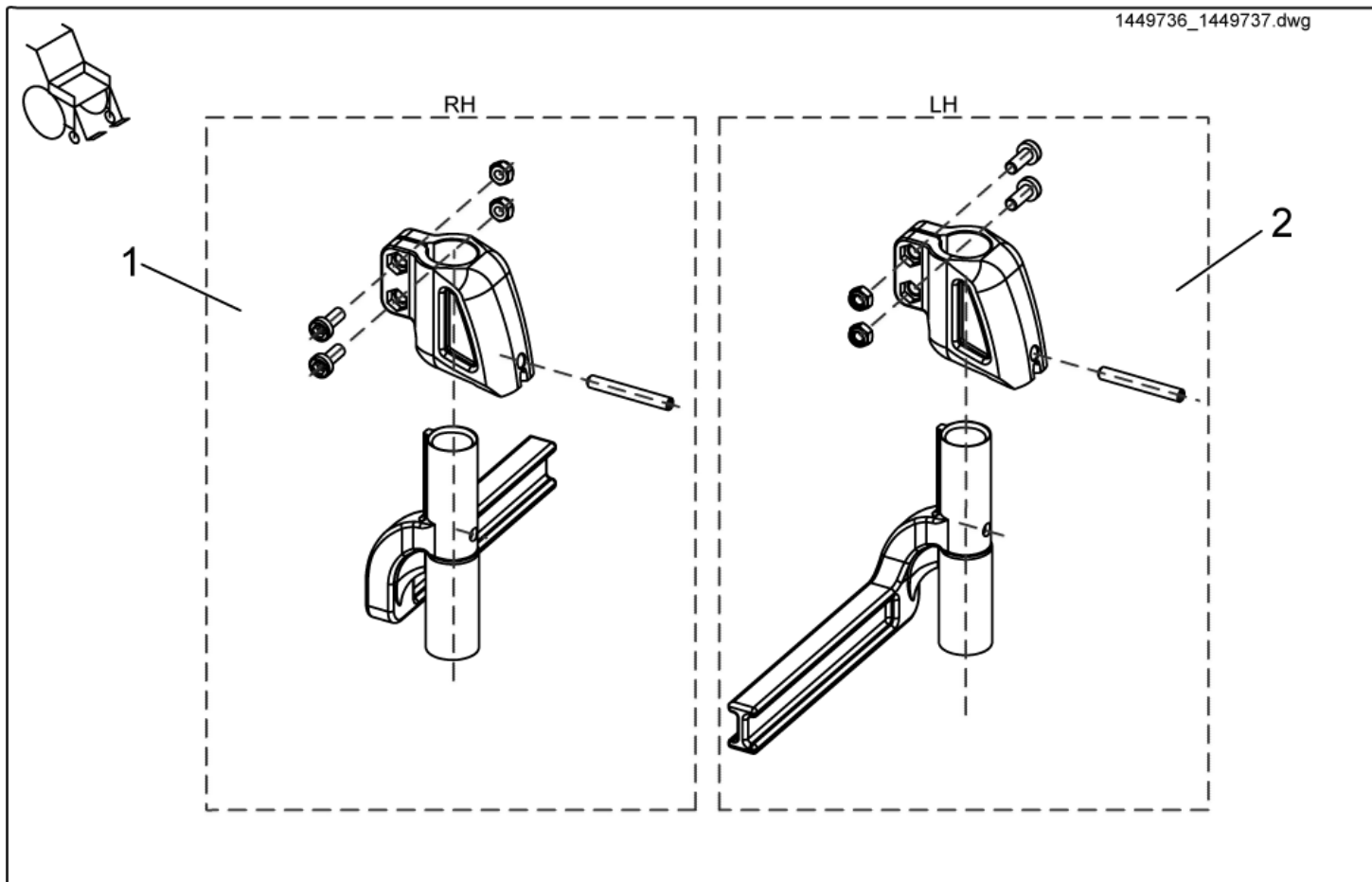1449736_1449737.dwg



item	ordercode	description	Menge
1	SP1449736	FOOTREST RECEPTACLE COMPLETE, RH MANUAL 8CM	1
2	SP1449737	FOOTREST RECEPTACLE COMPLETE, LH MANUAL 8CM	1

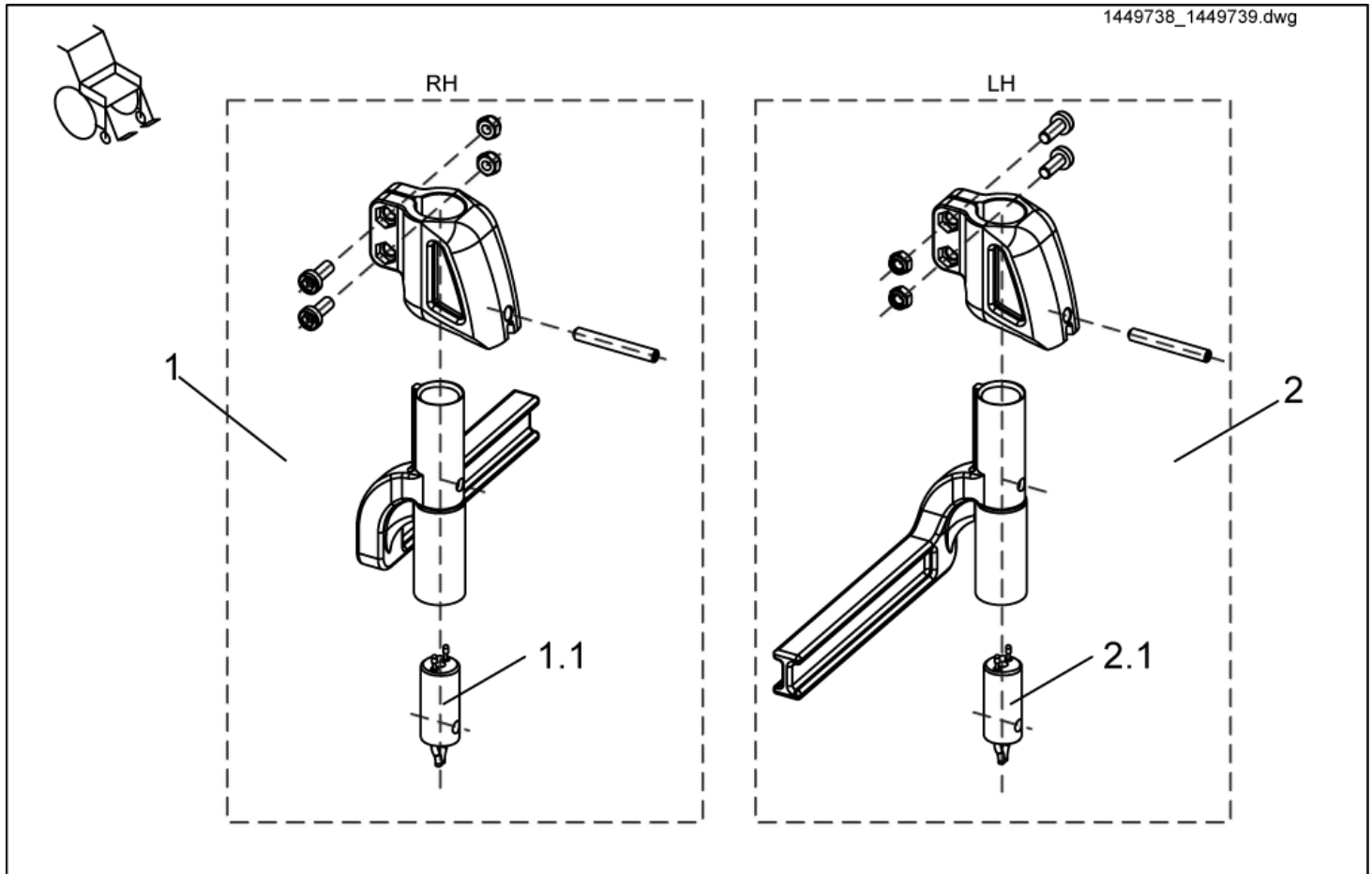1449738_1449739.dwg



item	ordercode	description	quantity
1	SP1449738	Footrest receptacle cpl., RH electric 8cm	1
1.1	1450228	Power supply cable footrest, short (black adapter)	1
2	SP1449739	Footrest receptacle cpl., LH electric 8cm	1
2.1	1450228	Power supply cable footrest, long (black adapter)	1

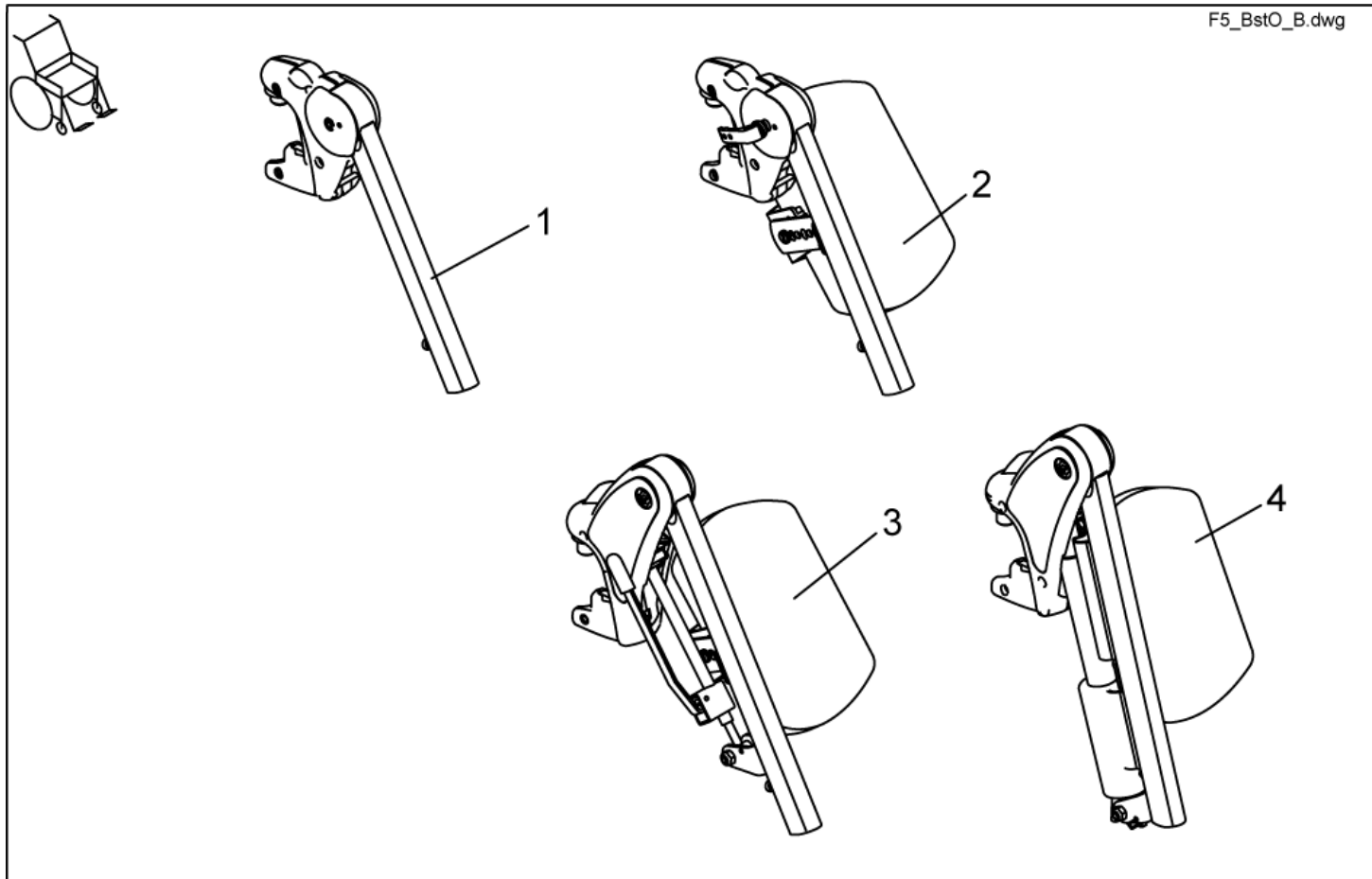


Note:

The Power supply cable, ordercode 1450228 and 1450232 are not compatible with Footrest receptacle starting from year of construction 07/2005.

Legrests, sideways mounted - Hanger


F5_BstO_B.dwg



item	ordercode	description	quantity
1	SP1422564	UPPER FOOTREST LH (90°-65°) CPL..	1
1	SP1422563	UPPER FOOTREST RH (90°-65°) CPL.	1
2	SP1422565	UPPER FOOTREST LH, MANUAL HEIGHT ADJUSTMENT, CPL.	1
2	SP1422566	UPPER FOOTREST RH, MANUAL HEIGHT ADJUSTMENT CPL.	1
3	SP1422568	UPPER FOOTREST LH 80°, MANUAL HEIGHT ADJUSTMENT, CPL.	1
3	SP1422567	UPPER FOOTREST T RH 80°, MANUAL HEIGHT ADJUSTMENT CPL.	1
4	SP1422570	UPPER FOOTREST LH, ELECTR. HEIGHT ADJUSTMENT, CPL.	1
4	SP1422569	UPPER FOOTREST RH, ELECTRIC HEIGHT ADJUSTMENT, CPL.	1


Legrests, sideways mounted - Lower Footrests

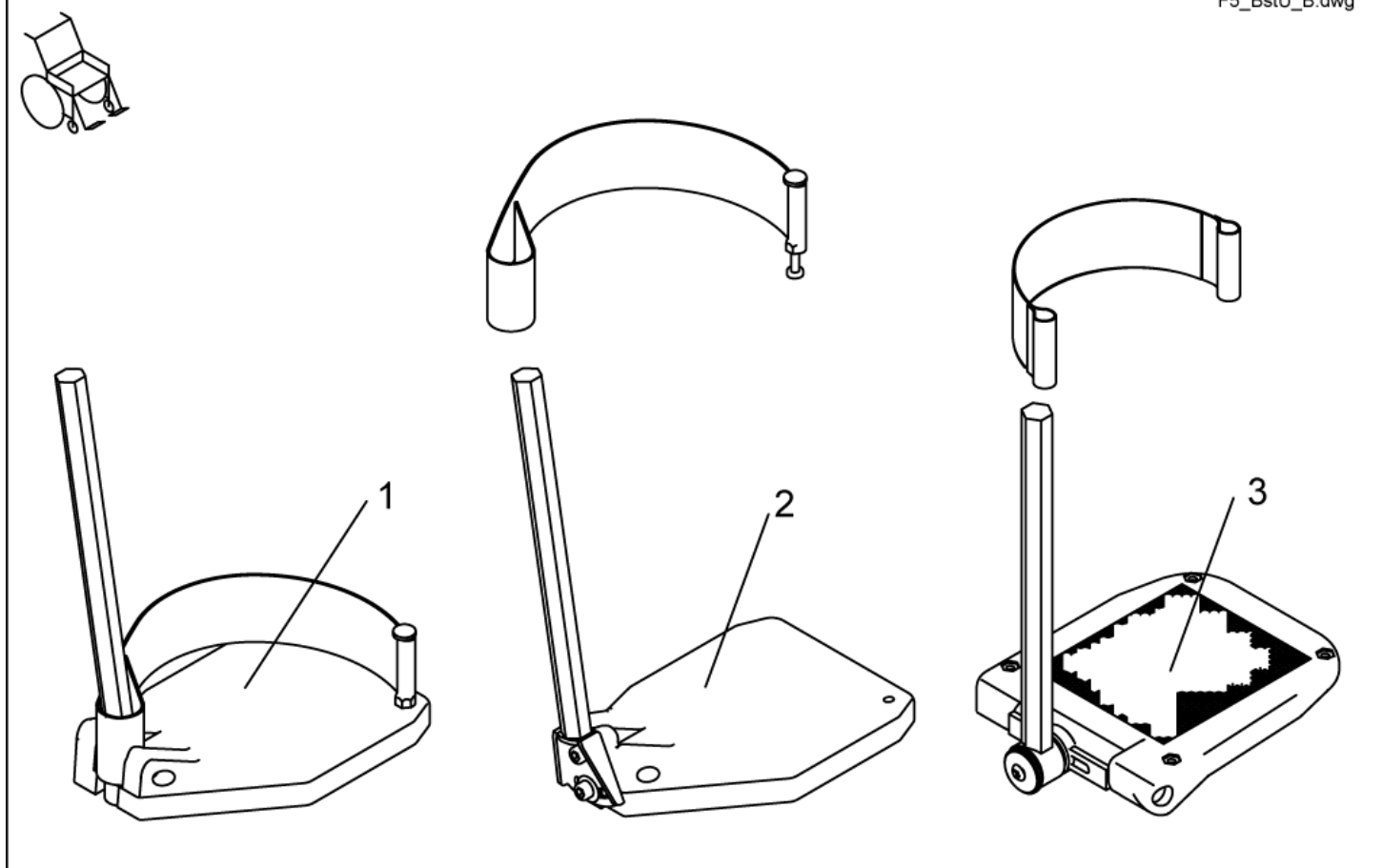F5_BstU_B.dwg





item	ordercode	description	quantity
1	SP1421663	LOWER FOOTREST LH, CPL.	1
1	SP1421662	LOWER FOOTREST RH, CPL.	1
2	SP1421665	LOWER FOOTREST, ANGLE ADJUSTMENT LH CPL.	1
2	SP1421664	LOWER FOOTREST, ANGLE ADJUSTMENT RH CPL.	1
3	SP1419344	FOOT PLATE LH ANGLE AND DEPTH ADJUSTMENT CPL.	1
3	SP1419345	FOOT PLATE RH ANGLE AND DEPTH ADJUSTMENT CPL.	1

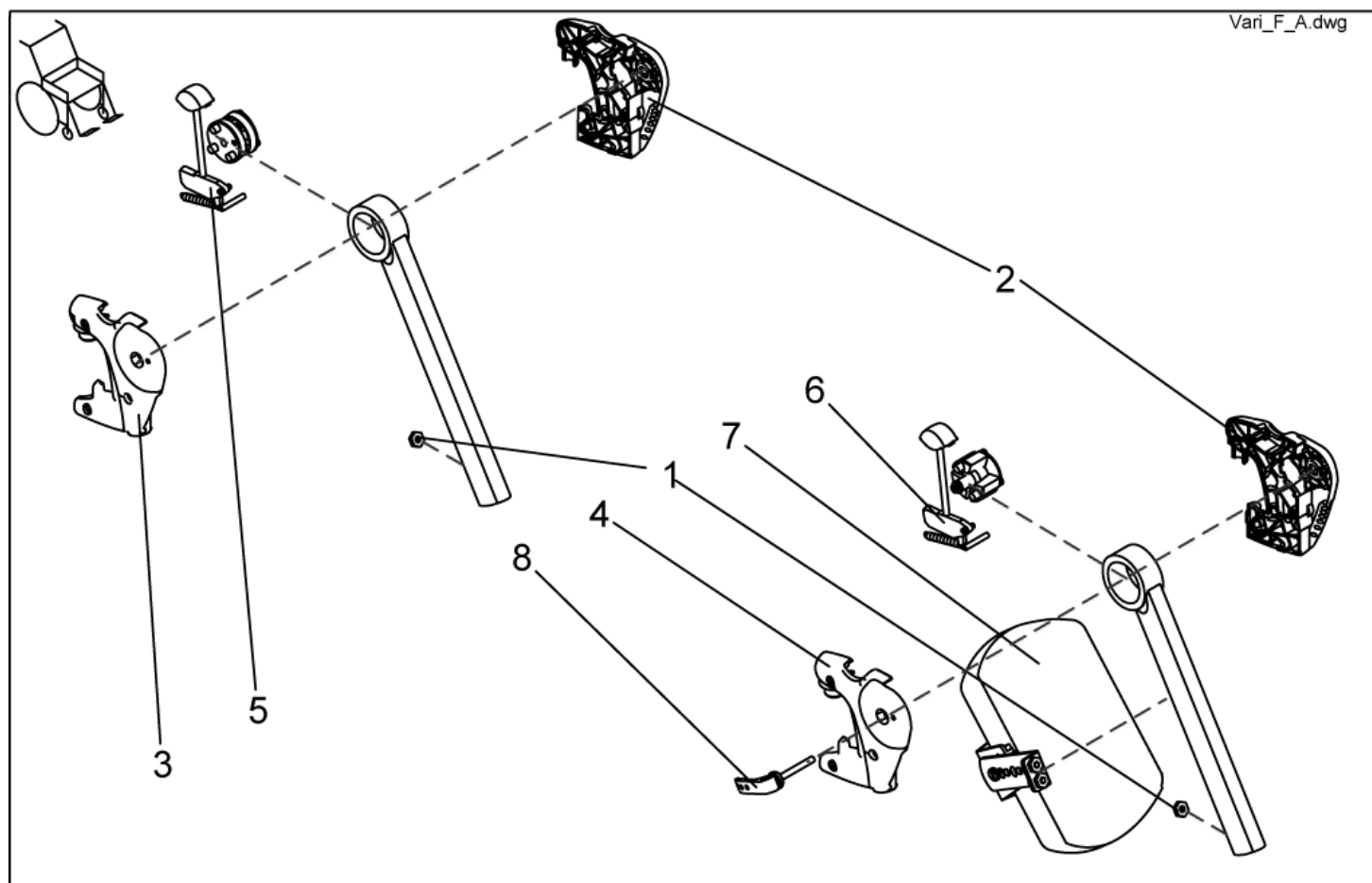
Legrests, sideways mounted - Vari F+A Detailed




item	ordercode	description	quantity
1	Z173189.99	NUT M6 SW-14	2
2	Z173181	HANGER LH	1
3	Z173182	HANGER RH	1
4	Z173194	HANGER RH	1
5	SP1423193	RELEASE UNIT ASSEMBLY KIT	2
6	SP1423194	RELEASE UNIT ASSEMBLY KIT F. LEGREST W. MAN. HEIGHT ADJUSTMENT	2
7	SPZ174163	CALF SUPPORT (VARI A) CPL.	2
7	6301333	CALF SUPPORT	2
8	L32906	CLAMPING LEVER M6	2

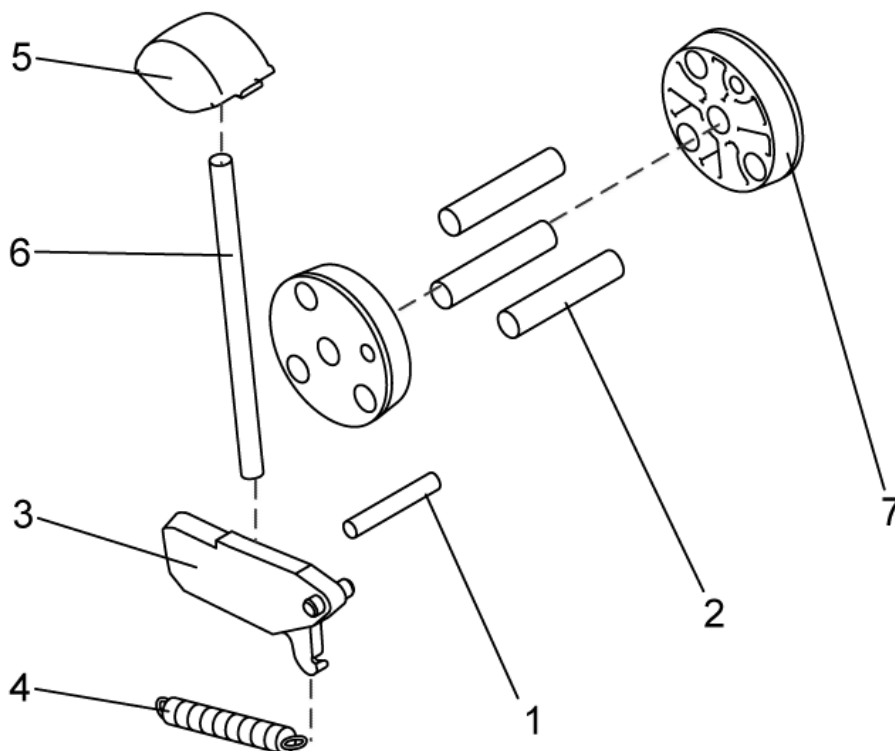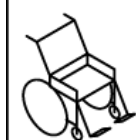


item	ordercode	description	quantity
1-7	SP1423193	RELEASE UNIT ASSEMBLY KIT	2
1	NA	DOWEL RD. 5 X 32; DIN-7	1
2	NA	DOWEL RD. 8 X 40; DIN-7	3
3	NA	LOCK LEVER	1
4	NA	SPRING	1
5	NA	RELEASE KNOB	1
6	NA	PUSH ROD	1
7	NA	CONED WASHER	2

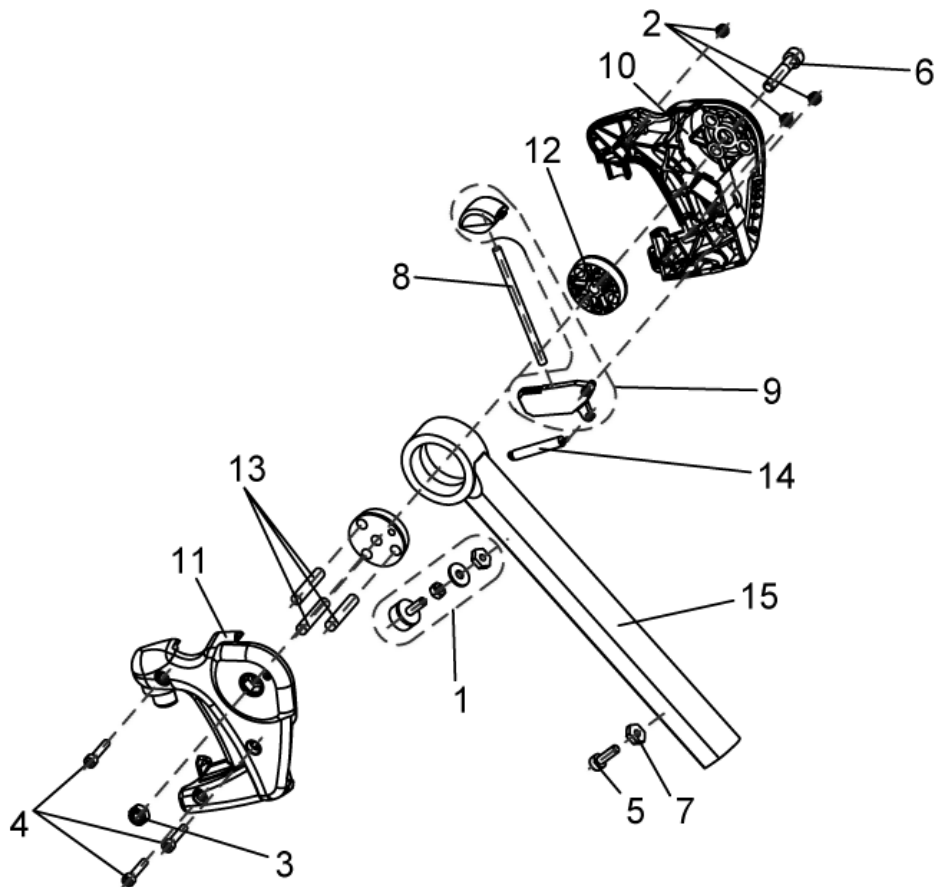


item	ordercode	description	quantity
-	SP1449346	UPPER VARI F, RH COMPLETE	1
1	1449154	RUBBER STOP COMPLETE, VARI A / VARI F	1
2	NA	NYLOC NUT M5	3
3	NA	NYLOC NUT M8	1
4	NA	SOCKET HEAD CAP SCREW M5 X 25	3
5	NA	SOCKET HEAD CAP SCREW M6 X 20	1
6	NA	SOCKET HEAD CAP SCREW M8 X 40	1
7	Z173189.99	NUT M6 SW-14	2
8	NA	PUSH ROD	1
9	1449167	RELEASE BUTTON + LOCK LEVER	1
10	Z173181	HANGER LH	1
11	Z173182	HANGER RH	1
12	Z173188	CONED WASHER	2
13	NA	ZYLINDERSTIFT RD. 8 X 40; DIN-7	3
14	Z173178	SPRING	1
15	Z173185-9005	UPPER FOOTREST RH (90-65°)	1

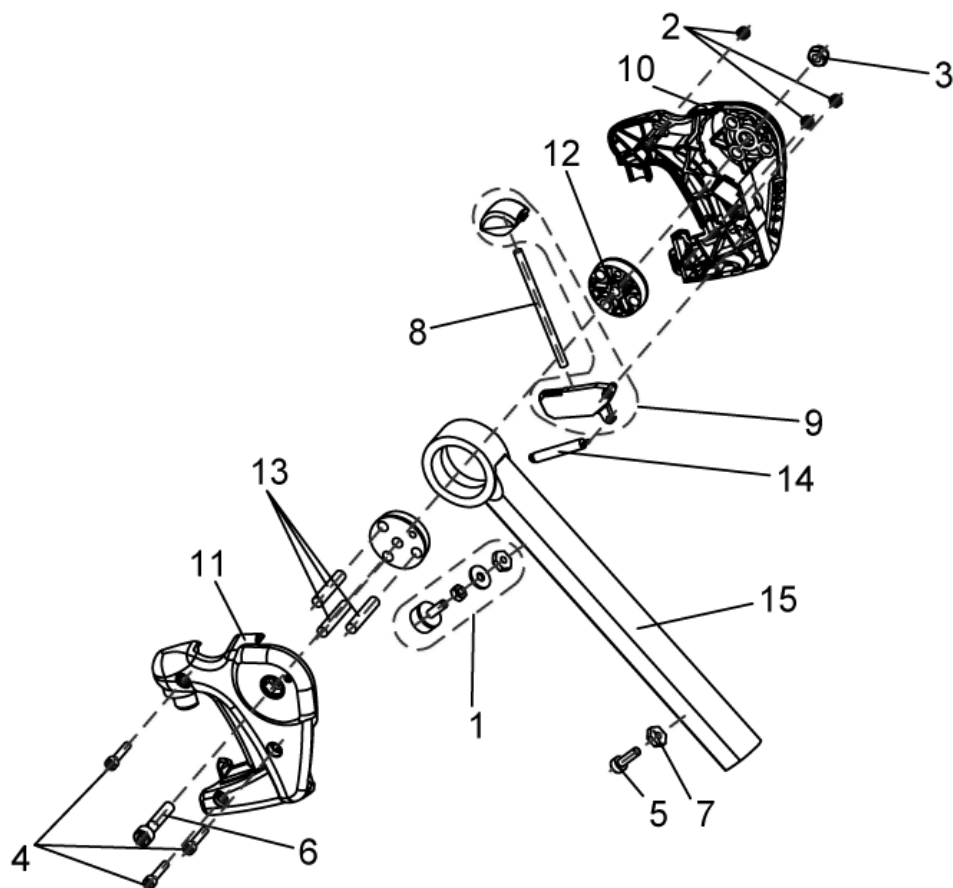


item	ordercode	description	quantity
-	SP1449347	UPPER VARI F, LH COMPLETE	1
1	1449154	RUBBER STOP COMPLETE, VARI A / VARI F	1
2	NA	NYLOC NUT M5	3
3	NA	NYLOC NUT M8	1
4	NA	SOCKET HEAD CAP SCREW M5 X 25	3
5	NA	SOCKET HEAD CAP SCREW M6 X 20	1
6	NA	SOCKET HEAD CAP SCREW M8 X 40	1
7	Z173189.99	M6 NUT, WIDTH ACROSS FLAT 14	2
8	NA	PUSH ROD	1
9	1449167	RELEASE BUTTON + LOCK LEVER	1
10	Z173181	HANGER LH	1
11	Z173182	HANGER RH	1
12	Z173188	CONED WASHER	2
13	NA	DOWEL RD. 8 X 40; DIN-7	3
14	Z173178	SPRING	1
15	Z173185-9005	UPPER FOOTREST RH (90-65°)	1

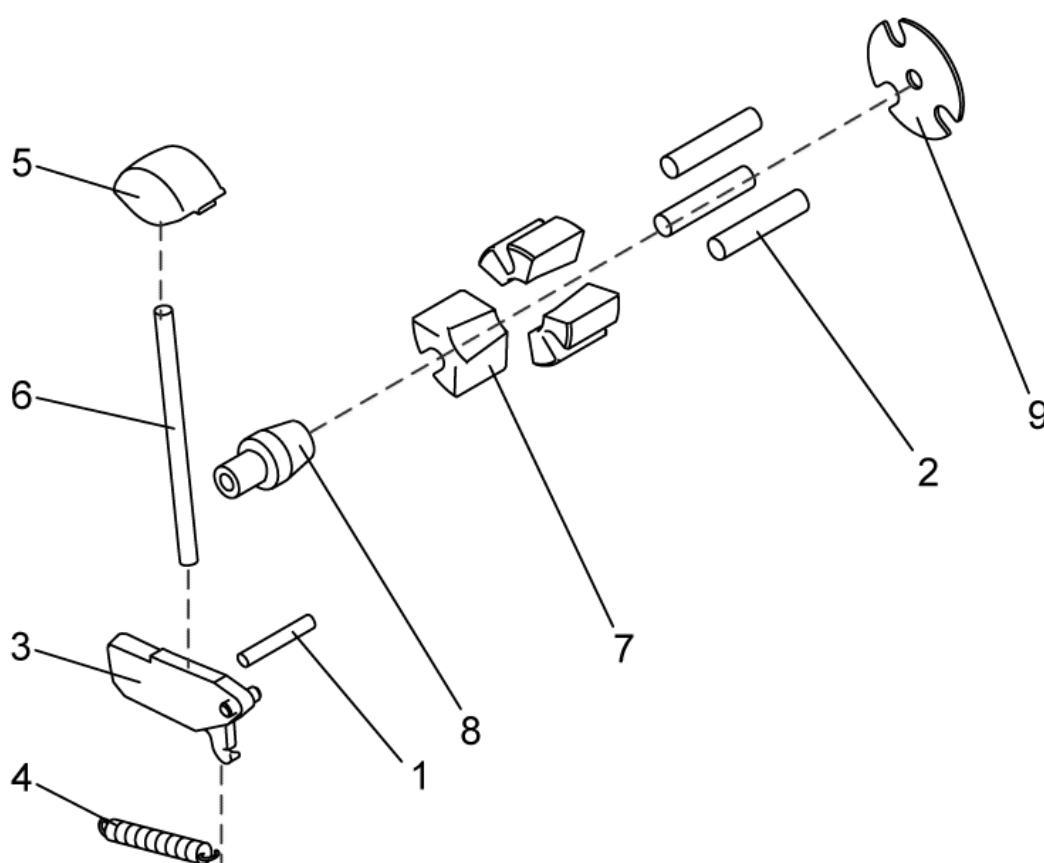

item	ordercode	description	quantity
1-9	SP1423194	RELEASE UNIT ASSEMBLY KIT F. LEGREST W. MAN. HEIGHT ADJUSTMENT	2
1	NA	DOWEL RD. 5 X 32; DIN-7	1
2	NA	DOWEL RD. 8 X 40; DIN-7	3
3	NA	LOCK LEVER	1
4	NA	SPRING	1
5	NA	RELEASE KNOB	1
6	NA	PUSH ROD	1
7	NA	CLAMP BLOCK	3
8	NA	CONE	1
9	NA	LOCATING WASHER	1

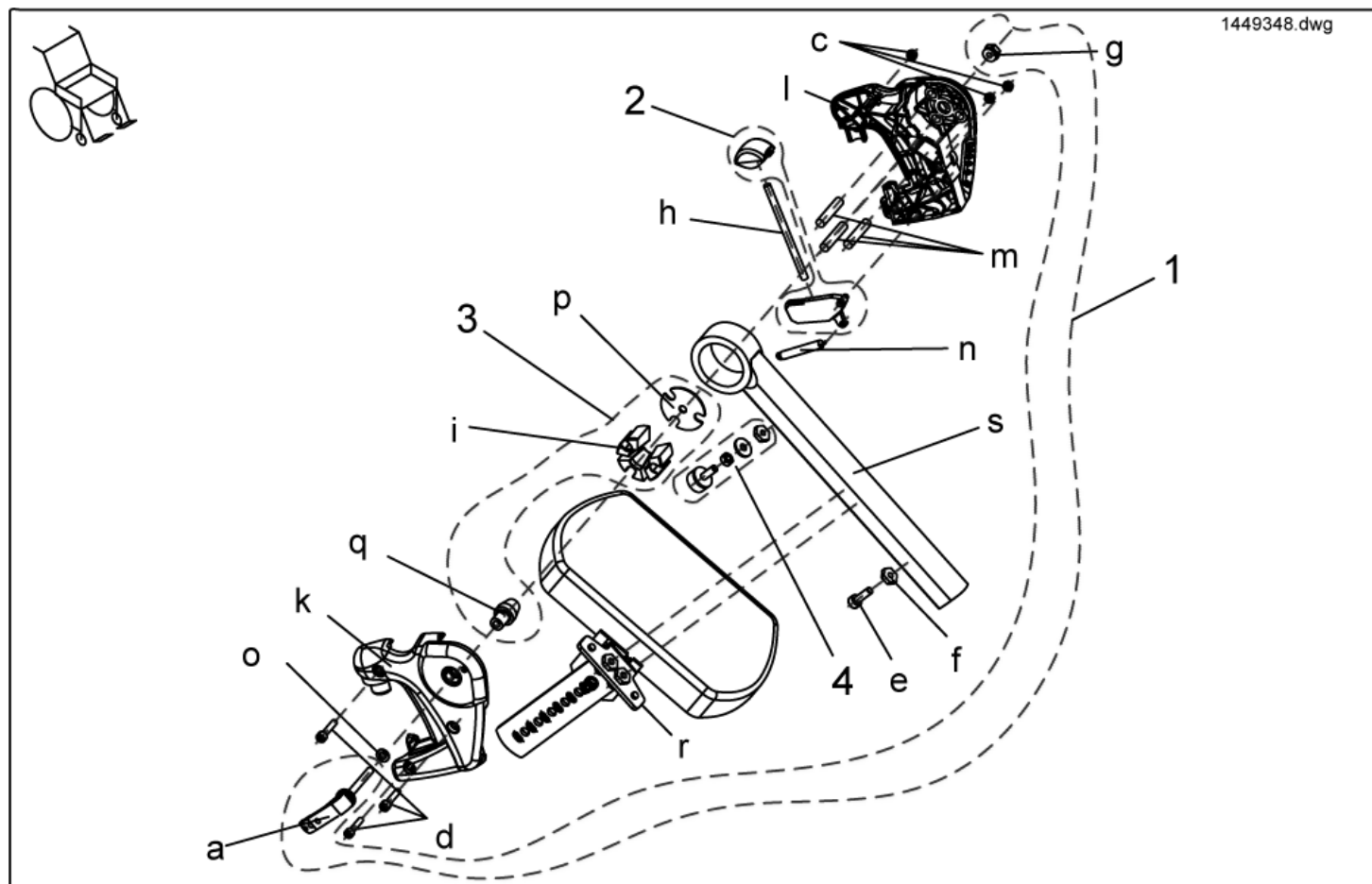


item	ordercode	description	quantity
-	SP1449348	UPPER VARI A COMPLETE, RH	1
1	1450597	ASSEMBLY KIT, VARI A RELEASE HANDLE	1
2	1449167	ASSEMBLY KIT, RELEASE BUTTON + LOCK LEVER	1
3	1450598	ASSEMBLY KIT, VARI A , CLAMPING BLOCK	1
4	1449154	ASSEMBLY KIT, RUBBER STOP COMPLETE, VARI A/VARI F	1
a	NA	CLAMP LEVER M6 X 25	1
c	NA	NYLOC NUT M5	3
d	NA	SOCKET HEAD CAP SCREW M5 X 25	3
e	NA	SOCKET HEAD CAP SCREW M6 X 20	1
f	NA	NUT M6 SW-14	2
g	NA	NUT M6 SW-13	1
h	NA	PUSH ROD	1
i	NA	CLAMP BLOCK	3
k	NA	Hanger, RH	1
l	NA	Hanger, LH	1
m	NA	DOWEL RD. 8 X 40; DIN-7	3
n	NA	SPRING	1
o	NA	WASHER 6,4 X 12,5 X 1,6	1
p	NA	LOCATING WASHER	1
q	NA	CONE	1
r	NA	CALF PLATE BRACKET RH, SWIVELLING	1
s	NA	UPPER FOOTREST VARI A	1

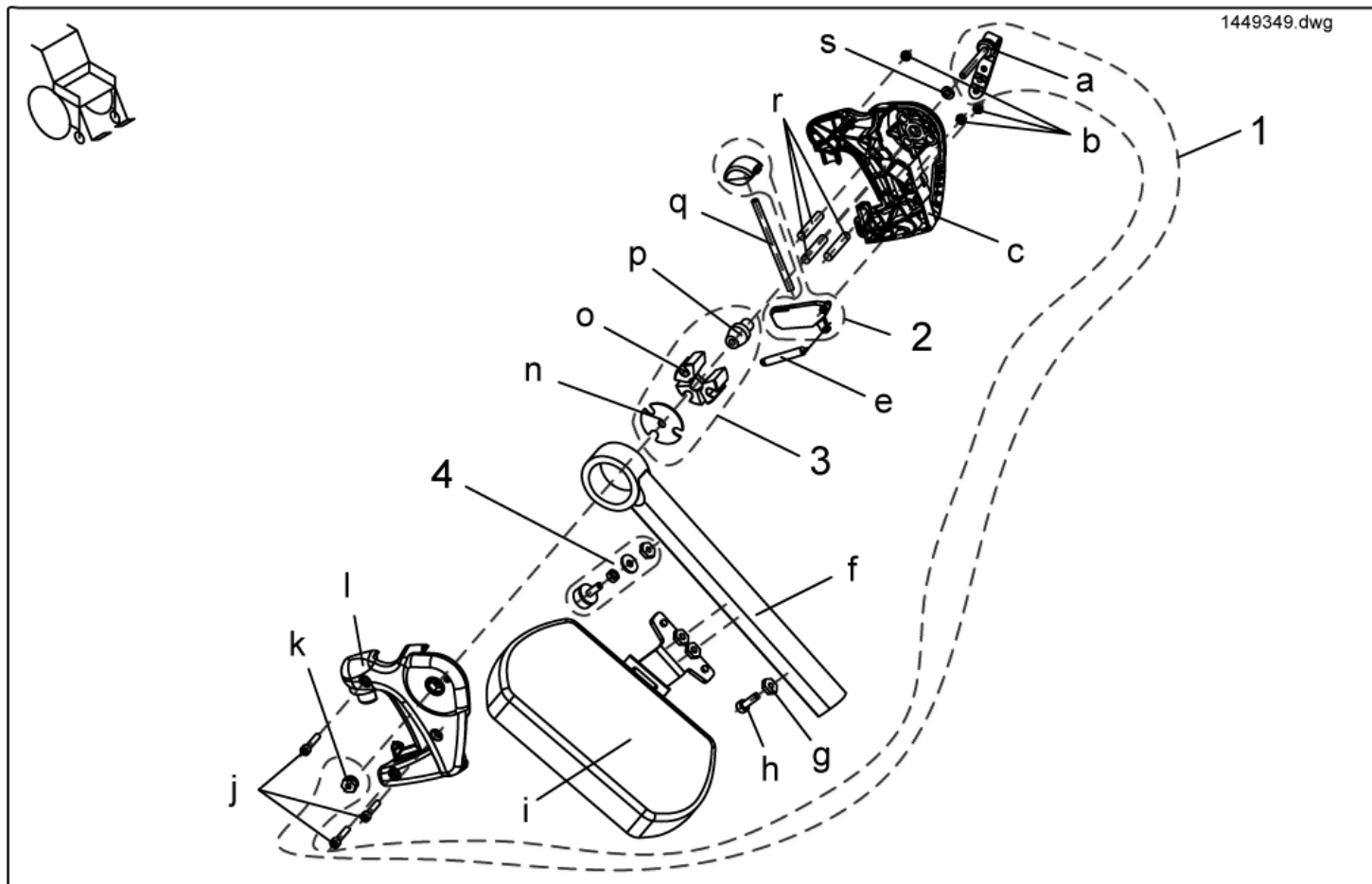


item	ordercode	description	quantity
-	SP1449349	UPPER VARI A COMPLETE, LH	1
1	1450597	ASSEMBLY KIT, VARI A RELEASE HANDLE	1
2	1449167	ASSEMBLY KIT, RELEASE BUTTON + LOCK LEVER	1
3	1450598	ASSEMBLY KIT, VARI A CLAMPING BLOCK	1
4	1449154	ASSEMBLY KIT, RUBBER STOP COMPLETE, VARI A / VARI F	1
a	NA	CLAMPING LEVER M6	1
b	NA	NYLOC NUT M5	3
c	NA	HANGER LH	1
e	NA	SPRING	1
f	NA	UPPER FOOTREST RH, MANUAL HEIGHT ADJUSTMENT	1
g	NA	NUT M6 SW-14	2
h	NA	SOCKET HEAD CAP SCREW M6 X 20	1
i	NA	CALF PLATE BRACKET, LH, SWIVELLING	1
j	NA	SOCKET HEAD CAP SCREW M5 X 25	3
k	NA	NUT M6 SW-13	1
l	NA	HANGER RH	1
n	NA	LOCATING WASHER	1
o	NA	CLAMP BLOCK	3
p	NA	CONE	1
q	NA	PUSH ROD	1
r	NA	DOWEL RD. 8 X 40; DIN-7	1
s	NA	WASHER 6,4 X 12,5 X 1,6	1

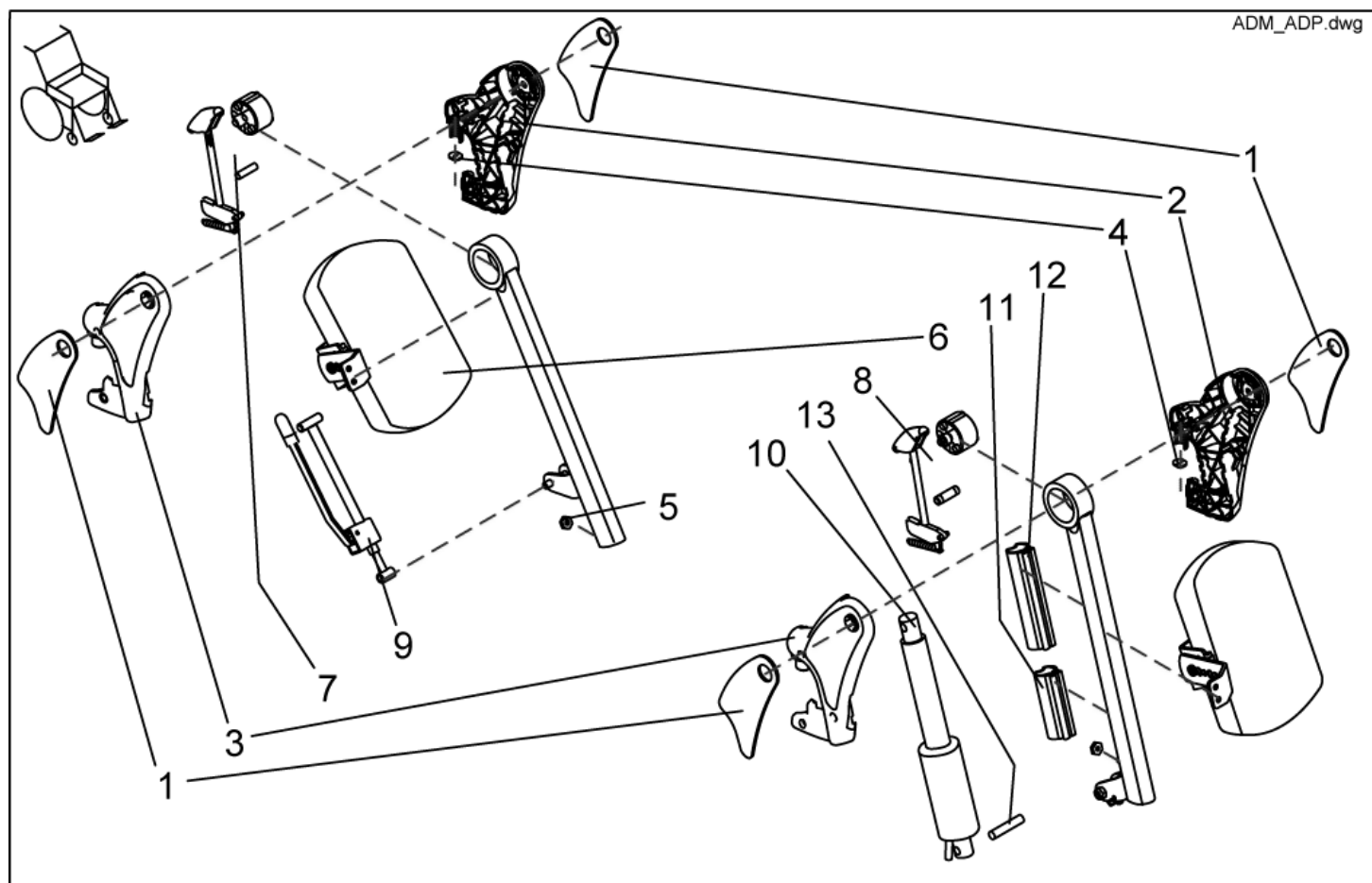
Legrests, sideways mounted - ADM + ADP Detailed




item	ordercode	description	quantity
1	1435024	KNEE PAD ADM (PAIR LH/RH)	2
2	Z174111	HANGER (AD) LH	1
3	Z174110	HANGER (AD) RH	1
4	Z174139	ADP-PLATE	2
5	Z173189.99	NUT M6 SW-14	6
6	SPZ174164	CALF SUPPORT ASSY (VARI AD)	2
6	6301333	CALF SUPPORT	2
7	SP1423195	RELEASE UNIT ASSEMBLY KIT F. LEGREST 80° W. MAN. HEIGHT ADJUSTMENT	2
8	SP1423201	RELEASE UNIT ASSEMBLY KIT F. LEGREST W. ELECTR. HEIGHT ADJUSTMENT	2
9	SPZ174130	ADM ADJUSTMENT MECHANISM CPL.	2
10	SPZ151006	ACTUATOR ASSY	2
11	Z174174	LOWER GUARD	2
12	Z174175	UPPER GUARD	2
13	4000665-0002	SPACER BUSH 10X1X22,5	2

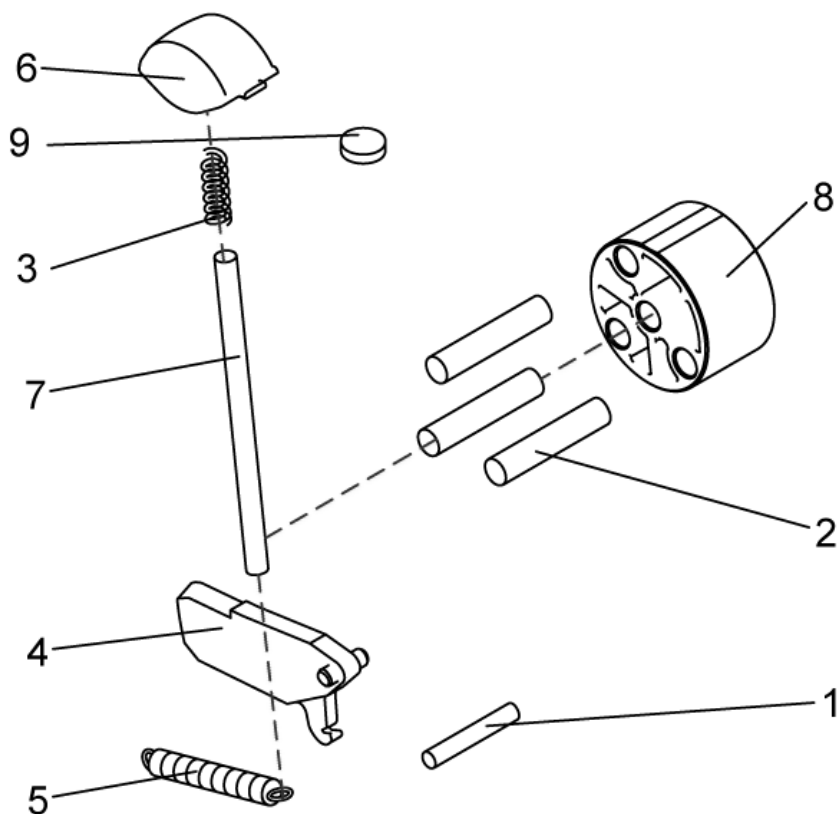

item	ordercode	description	quantity
1-9	SP1423195	RELEASE UNIT ASSEMBLY KIT F. LEGREST 80° W. MAN. HEIGHT ADJUSTMENT	2
1	NA	DOWEL RD. 8 X 30, DIN-7	1
2	NA	DOWEL RD. 8 X 36, DIN-7	3
3	NA	COMPRESSION SPRING	1
4	NA	LOCK LEVER	1
5	NA	SPRING	1
6	NA	RELEASE LEVER (AD)	1
7	NA	PUSH ROD	1
8	NA	SLIDING BUSH	1
9	Z174139	ADP-PLATE	1

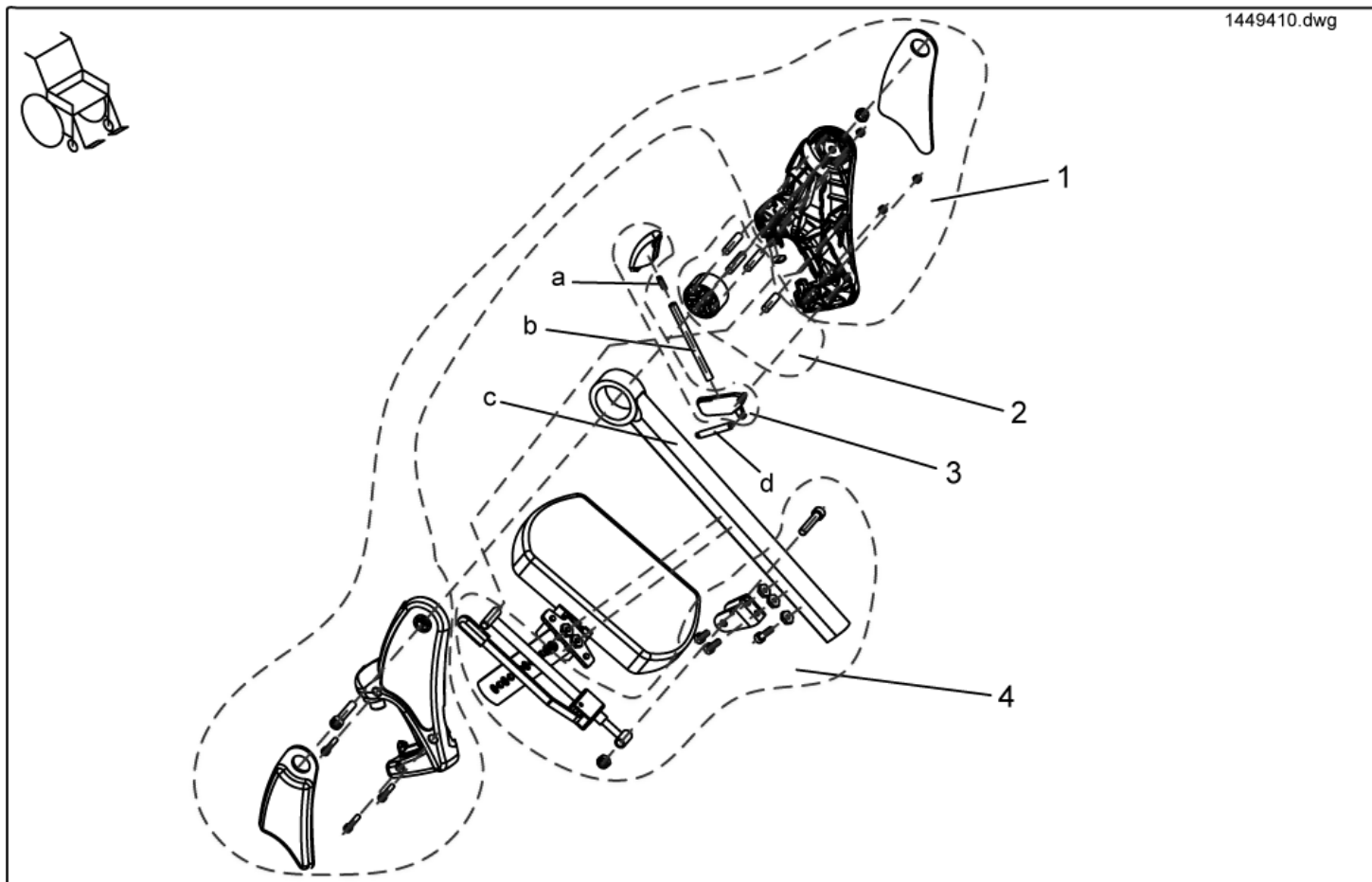


item	ordercode	description	quantity
-	SP1449410	COMPLETE UPPER ADM, RIGHT	1
1	NA	>>>see details on "assembly kit, hangers"	1
2	NA	>>>see details on "assembly kit, sliding bush"	1
3	NA	>>>see details on "trip lever + lock lever"	1
4	NA	>>>see details on "assembly kit, ADM adjustment"	1
a	NA	COMPRESSION SPRING	1
b	NA	PUSH ROD	1
c	Z173207-9005	UPPER FOOTREST T RH 80°, MANUAL HEIGHT ADJUSTMENT	1
d	NA	SPRING	1

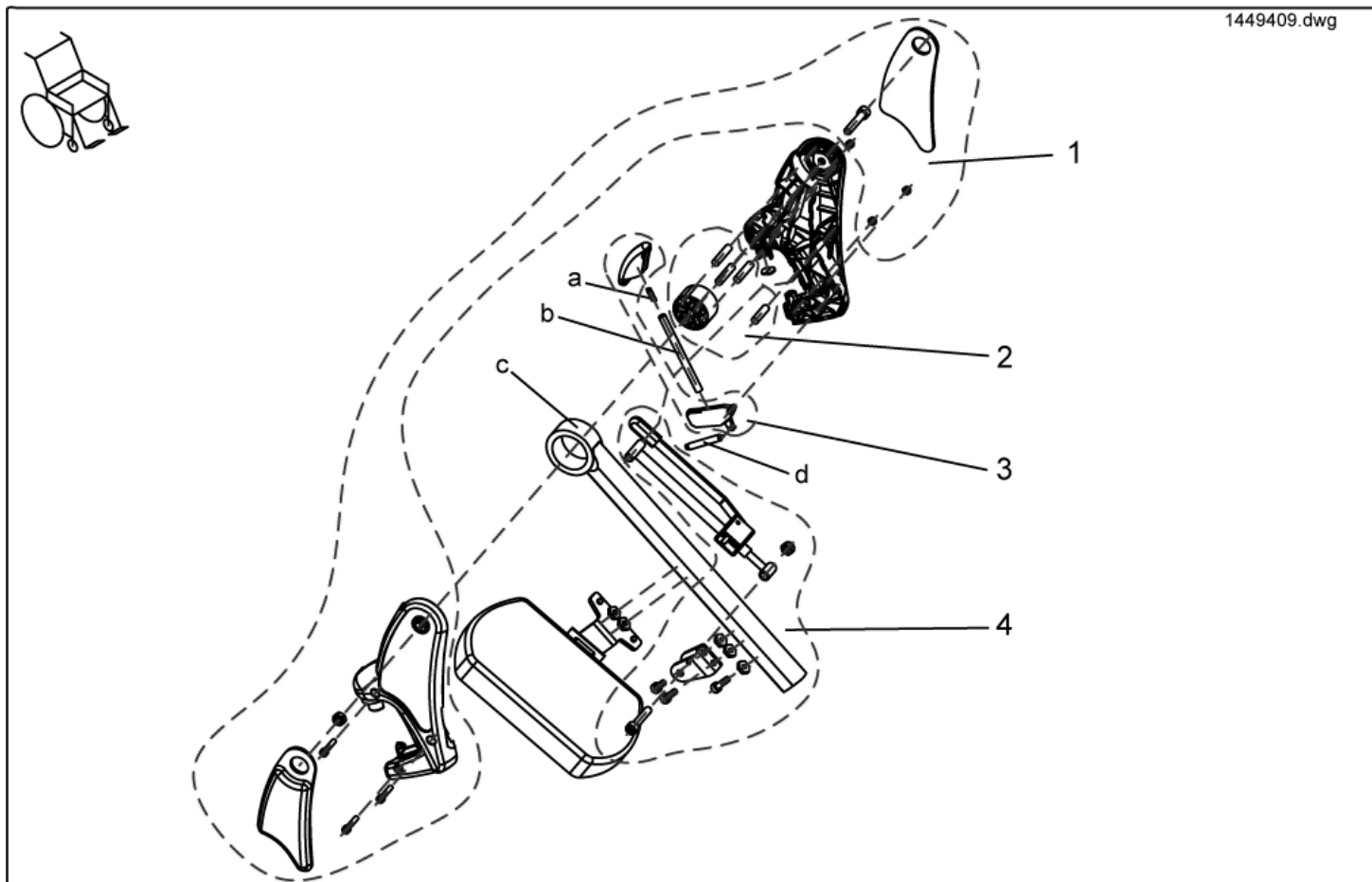

item	ordercode	description	quantity
-	SP1449409	Complete upper ADM, LH	1
1	NA	>>>see details on "assembly kit, hangers"	1
2	NA	>>>see details on "assembly kit, sliding bush"	1
3	NA	>>>see details on trip lever + lock lever "	1
4	NA	>>>see details on "assembly kit, ADM adjustment"	1
a	N12196	COMPRESSION SPRING	1
b	Z174113.99	BARRE DE FERMETURE	1
c	Z173207-9005	UPPER FOOTREST T RH 80°, MANUAL HEIGHT ADJUSTMENT	1
d	Z173178	SPRING	1

Legrests, sideways mounted - Assembly kit, hangers

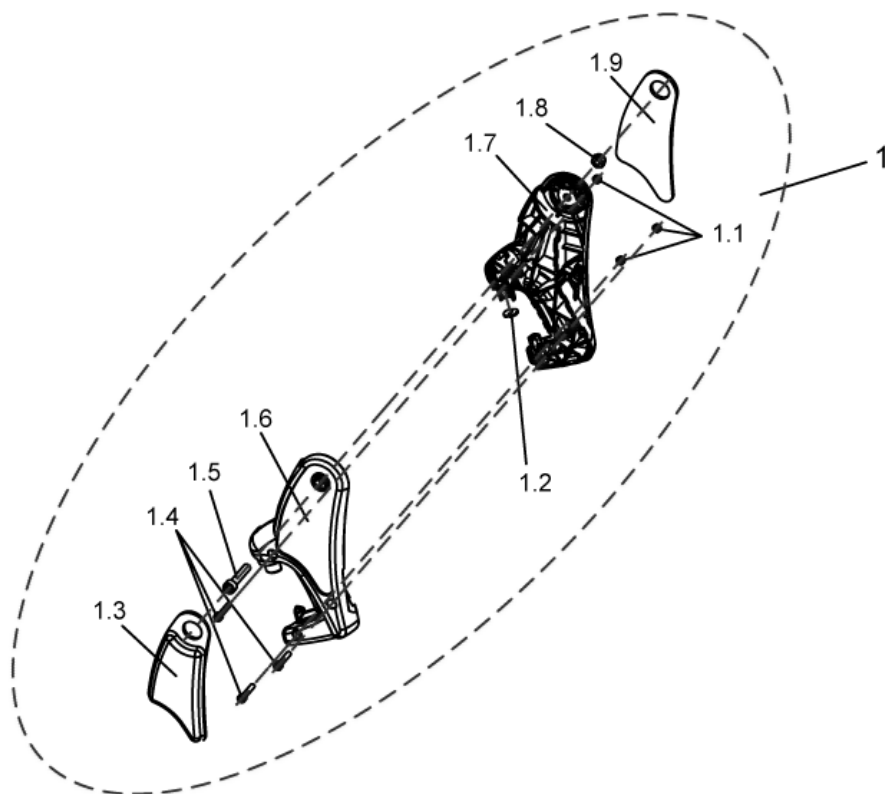
1449410_1.dwg



item	ordercode	description	quantity
1	1485604	ASSEMBLY KIT, ADM HANGERS	1
1.1	NA	NYLOC NUT M5	3
1.2	NA	ADP-PLATE	1
1.3	1450448	KNEE CUSHION, RH	1
1.4	NA	SOCKET HEAD CAP SCREW M5 X 25	3
1.5	NA	SOCKET HEAD CAP SCREW M8 X 40	2
1.6	NA	HANGER, RH	1
1.7	NA	HANGER, LH	1
1.8	NA	NYLOC NUT M8	2
1.9	1450410	KNEE CUSHION, LH	1

Legrests, sideways mounted - Assembly kit, ADM sliding bush

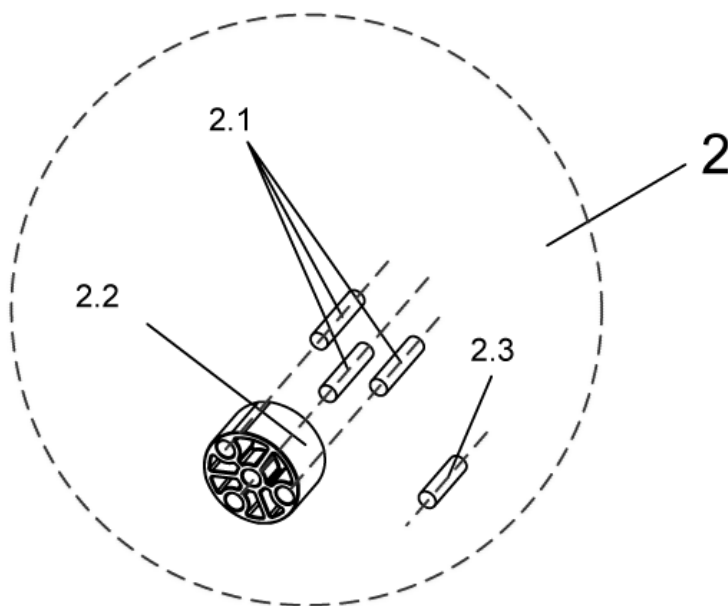
1449410_2.dwg



item	ordercode	description	quantity
2	1485605	ASSEMBLY KIT, ADM SLIDING BUSH	1
2.1	NA	DOWEL RD. 8 X 36, DIN-7	3
2.2	NA	SLIDING BUSH	1
2.3	NA	DOWEL RD. 8 X 30, DIN-7	1

Legrests, sideways mounted - Assembly kit, Trip lever + lock lever

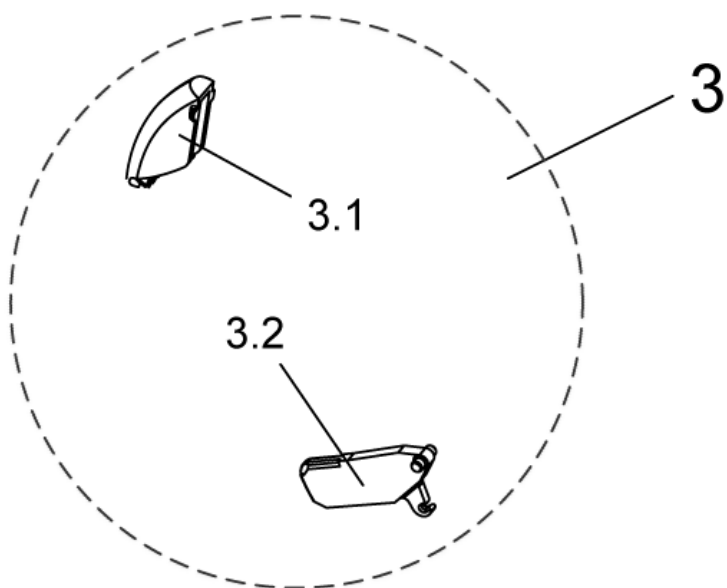
1449410_3.dwg




item	ordercode	description	quantity
3	1449155	TRIP LEVER + LOCK LEVER	1
3.1	NA	TRIP LEVER	1
3.2	NA	LOCK LEVER	1

Legrests, sideways mounted - Assembly kit, ADM adjustment

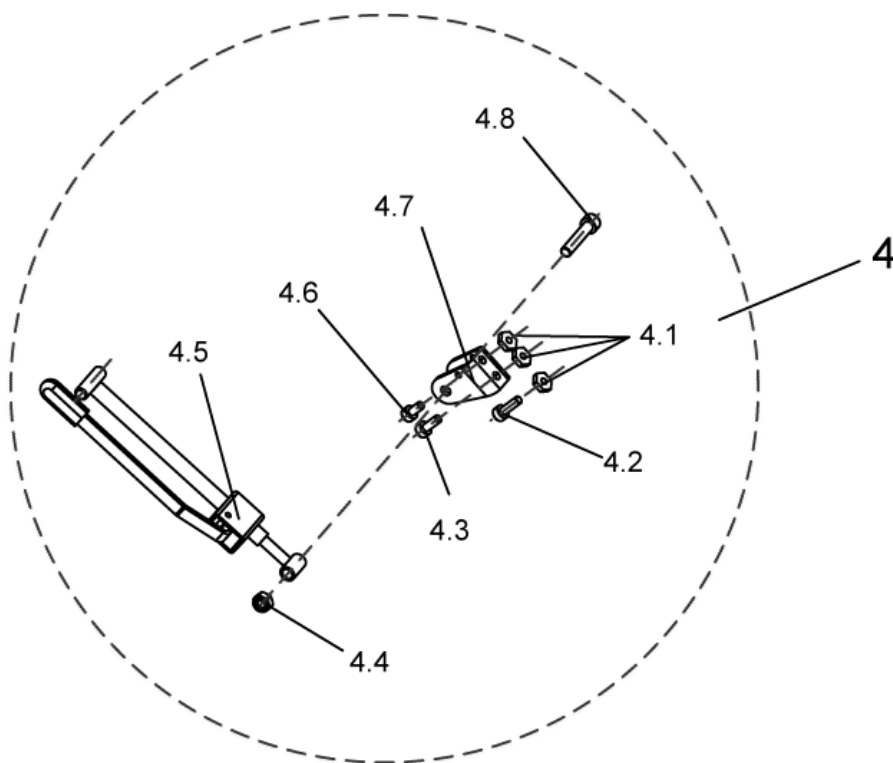1449410_4.dwg



item	ordercode	description	quantity
4	1485606	ASSEMBLY KIT, ADM ADJUSTMENT	1
4.1	NA	NUT M6 SW-14	3
4.2	NA	SOCKET HEAD CAP SCREW M6 X 20	1
4.3	NA	SOCKET HEAD CAP SCREW M6 X 16	1
4.4	NA	NYLOC NUT M8	2
4.5	NA	ADM ADJUSTMENT MECHANISM CPL.	1
4.6	NA	SOCKET HEAD CAP SCREW M6 X 12	1
4.7	NA	ACTUATOR MOUNT	1
4.8	NA	SOCKET HEAD CAP SCREW M8 X 40	2

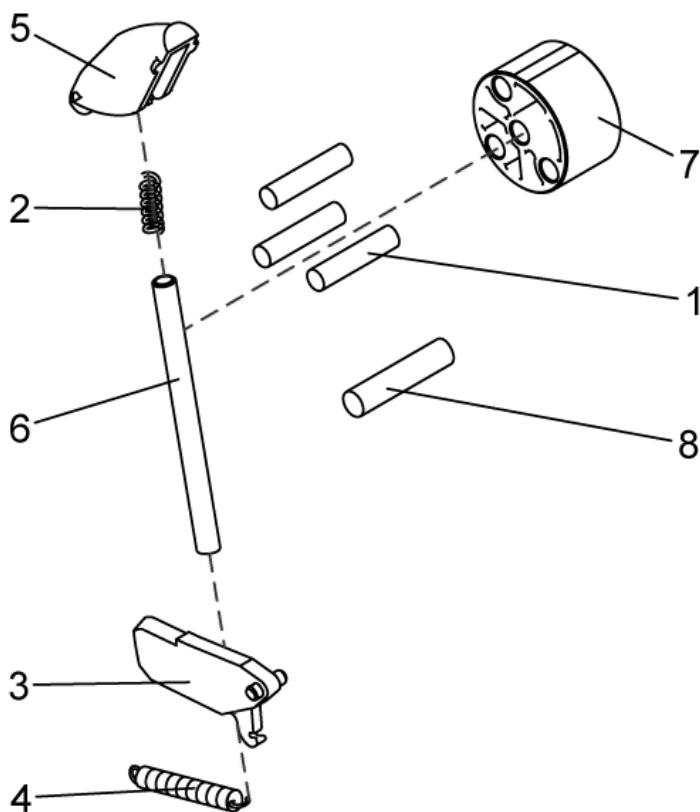

item	ordercode	description	quantity
1-8	SP1423201	RELEASE UNIT ASSEMBLY KIT F. LEGREST W. ELECTR. HEIGHT ADJUSTMENT	2
1	NA	DOWEL RD. 8 X 36, DIN-7	3
2	NA	COMPRESSION SPRING	1
3	NA	LOCK LEVER	1
4	NA	SPRING	1
5	NA	RELEASE LEVER (AD)	1
6	NA	PUSH ROD	1
7	NA	SLIDING BUSH	1
8	NA	PIVOT BUSH	1

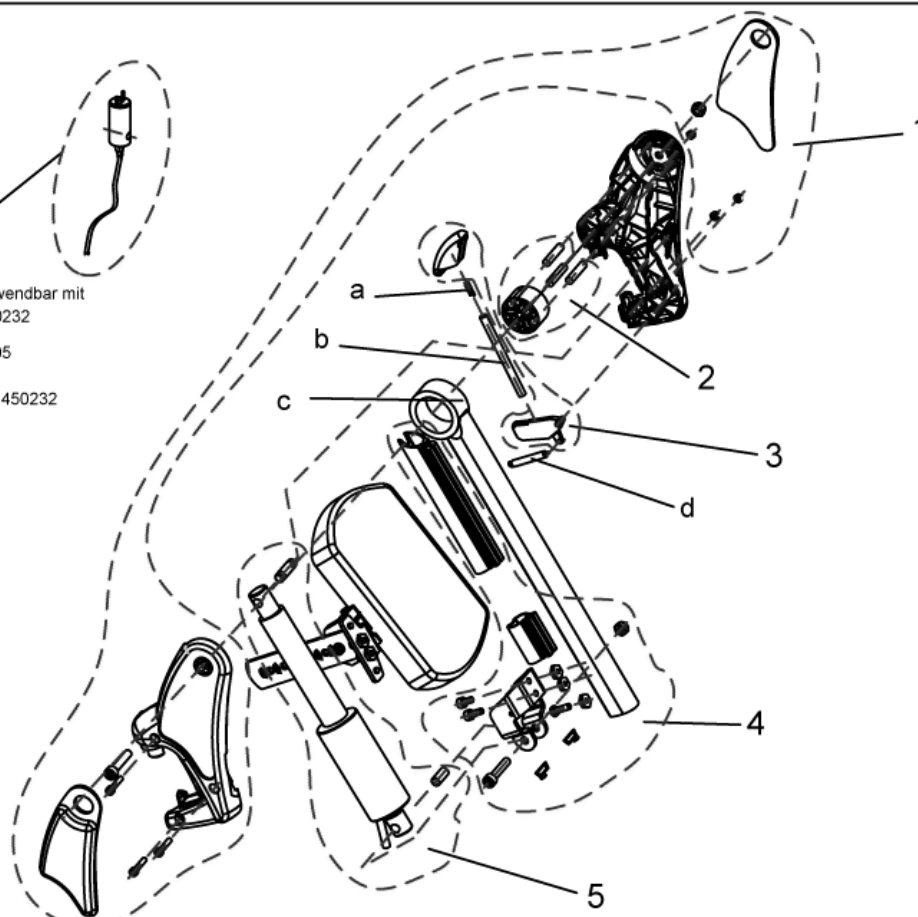
Legrests, sideways mounted - Upper ADP, right 07/2005 ==>

1449408.dwg




SP1449408 ab 07/05 nur verwendbar mit
Artikel-Nr.: 1450228 und 1450232

SP1449408 starting from 07/05
only usable together with
article number 1450228 and 1450232




item	ordercode	description	quantity
-	SP1449408	UPPER ADP, RH	1
1	NA	>>>see details on "assembly kit, hangers"	1
2	NA	>>>see details on "assembly kit ADP, sliding bush"	1
3	NA	>>>see details on "assembly kit ADP, release button + lock lever"	1
4	NA	>>>see details on "assembly kit ADP, actuator bracket"	1
5	NA	>>>see details on "assembly kit ADP, hoist motor"	1
a	N12196	COMPRESSION SPRING	1
b	Z174113.99	PUSH ROD	1
c	NA	UPPER FOOTREST, ADP	1
d	Z173178	SPRING	1

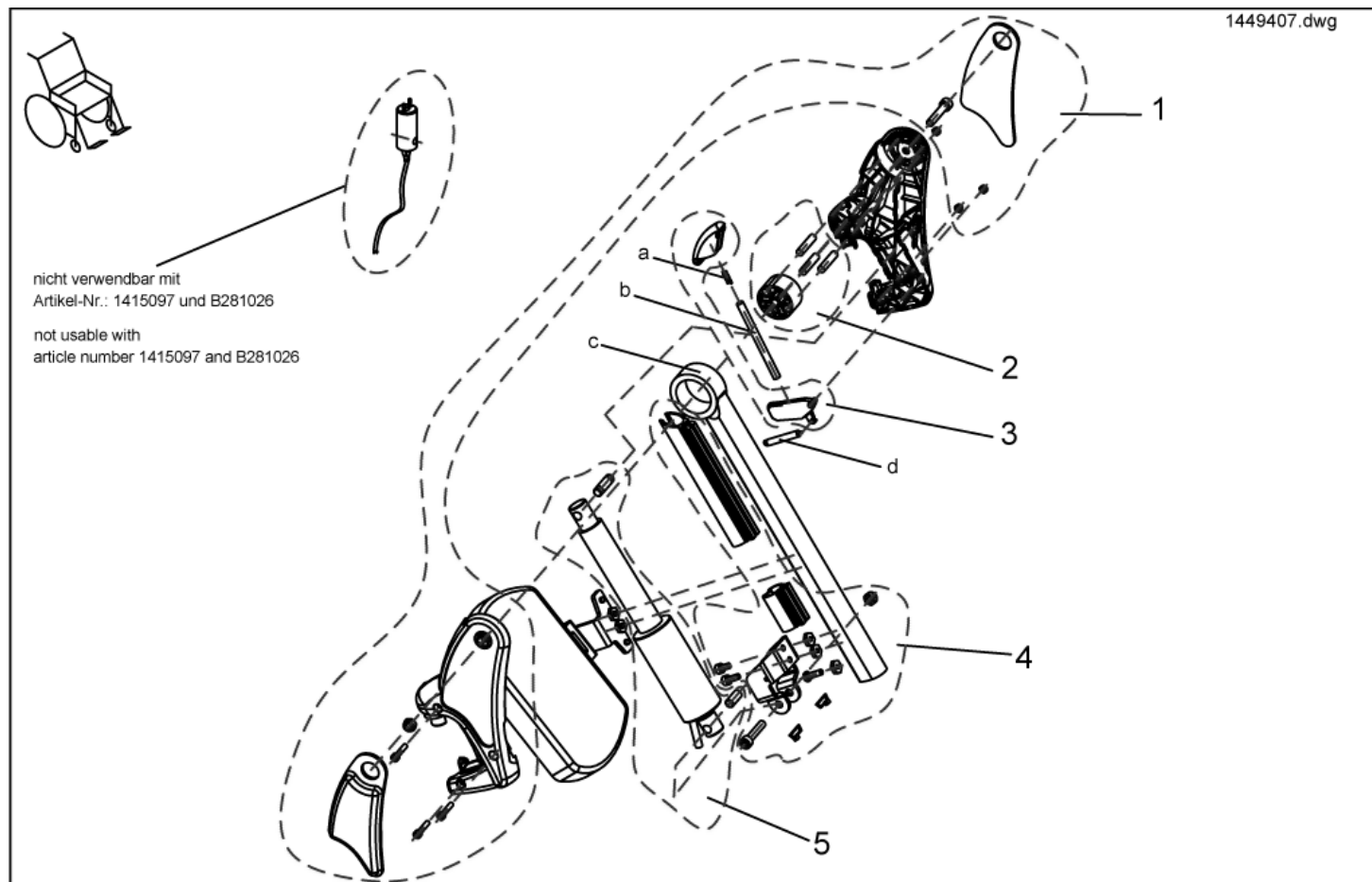

item	ordercode	description	quantity
-	SP1449407	UPPER ADP LH	1
1	NA	>>>see details on "assembly kit, hangers"	1
2	NA	>>>see details on "assembly kit ADP, sliding bush"	1
3	NA	>>>see details on "assembly kit ADP, trip lever + lock lever"	1
4	NA	>>>see details on "assembly kit ADP, actuator bracket"	1
5	NA	>>>see details on "assembly kit ADP, hoist motor"	1
a	1450410	KNEE CUSHION, LH	1
b	Z174113.99	PUSH ROD	1
c	NA	UPPER FOOTREST, ADP	1
d	Z173178	TENSION SPRING	1

Legrests, sideways mounted - Assembly kit, ADM Hangers

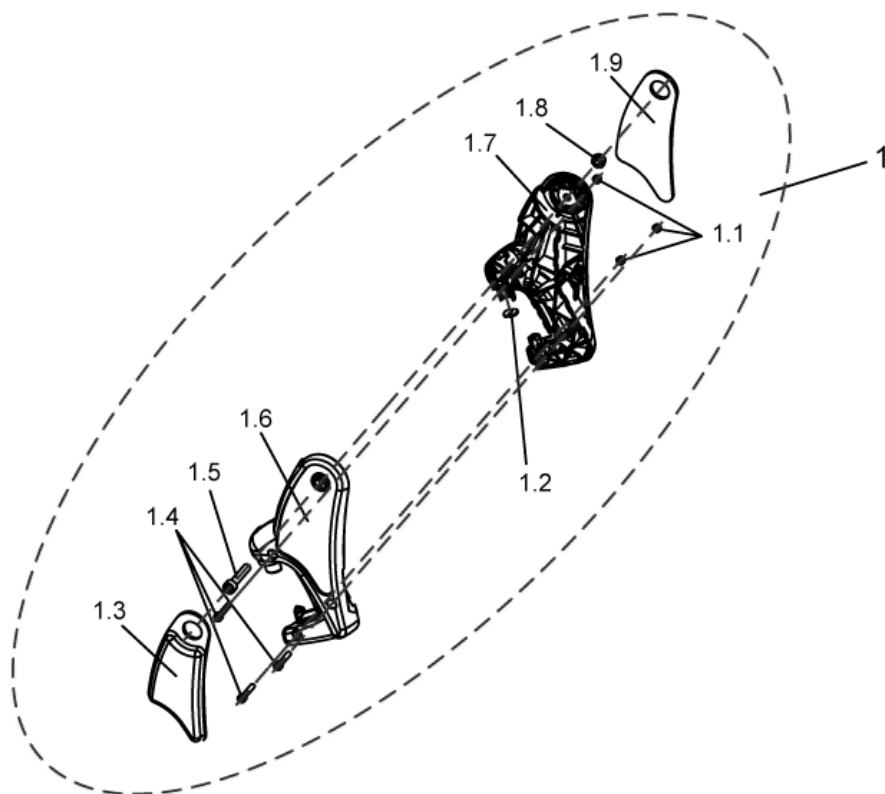
1449410_1.dwg



item	ordercode	description	quantity
1	1485604	ASSEMBLY KIT, ADM HANGERS	1
1.1	NA	NYLOC NUT M5	3
1.2	NA	ADP-PLATE	1
1.3	1450448	KNEE CUSHION, RH	1
1.4	NA	SOCKET HEAD CAP SCREW M5 X 25	3
1.5	NA	SOCKET HEAD CAP SCREW M8 X 40	2
1.6	NA	HANGER, RH	1
1.7	NA	HANGER, LH	1
1.8	NA	NYLOC NUT M8	2
1.9	1450410	KNEE CUSHION, LH	1

Legrests, sideways mounted - Assembly kit, ADM sliding bush

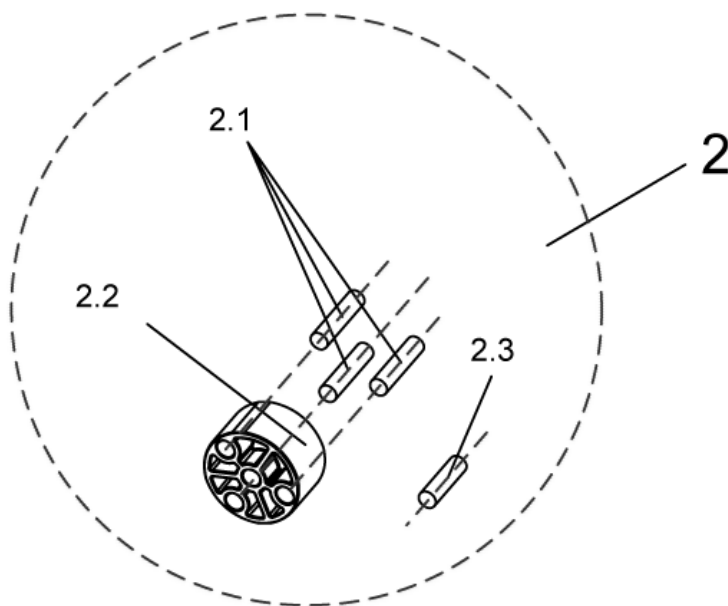
1449410_2.dwg



item	ordercode	description	quantity
2	1485605	ASSEMBLY KIT, ADM SLIDING BUSH	1
2.1	NA	DOWEL RD. 8 X 36, DIN-7	3
2.2	NA	SLIDING BUSH	1
2.3	NA	DOWEL RD. 8 X 30, DIN-7	1

Legrests, sideways mounted - Assembly, Trip lever + Lock lever

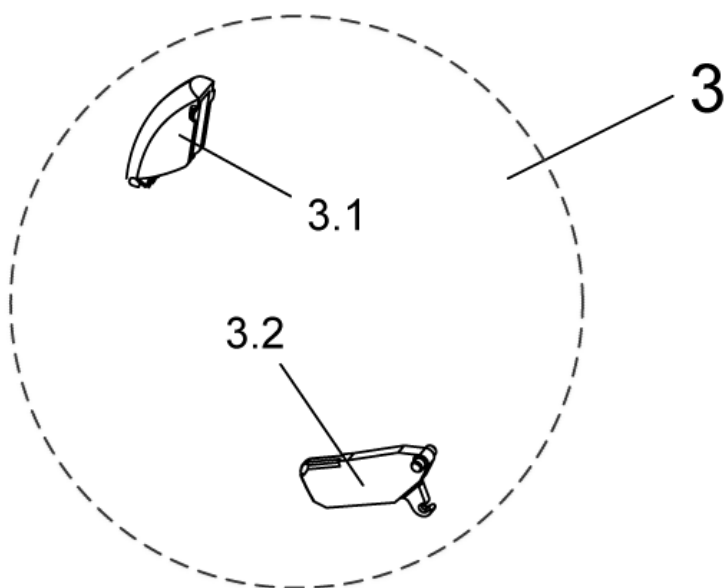
1449410_3.dwg



item	ordercode	description	quantity
3	1449167	TRIP LEVER + LOCK LEVER	1
3.1	NA	TRIP LEVER	1
3.2	NA	LOCK LEVER	1

Legrests, sideways mounted - Assembly kit ADP, Actuator Bracket

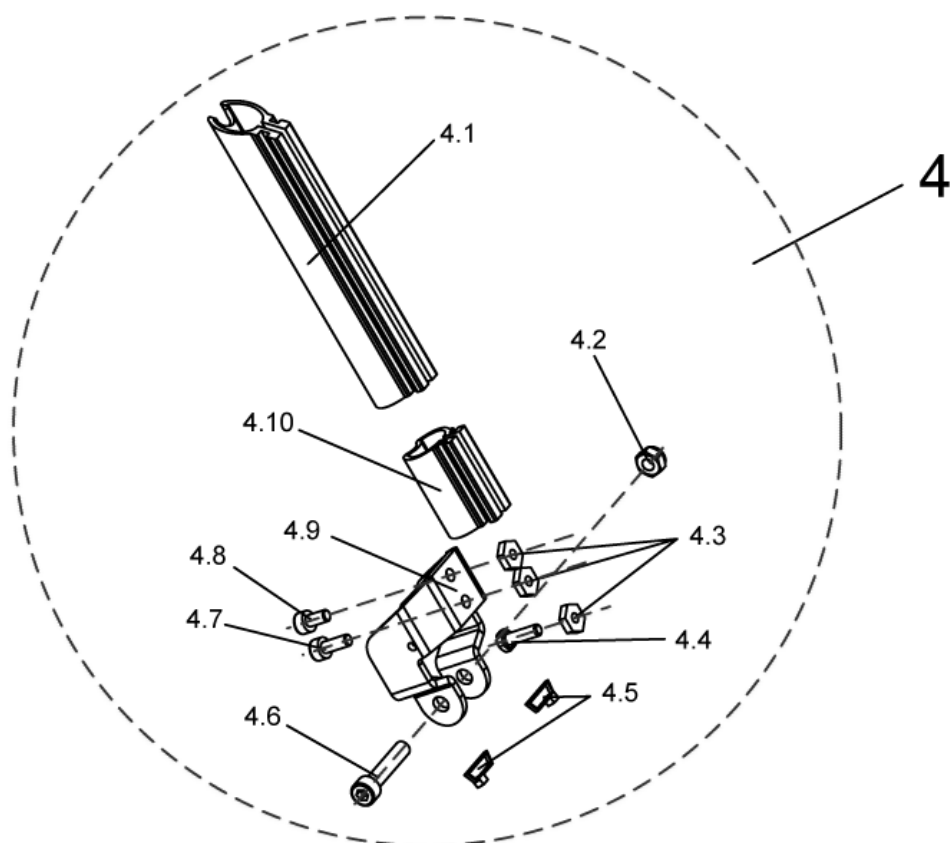
1449408_1.dwg



item	ordercode	description	quantity
4	1450607	ASSEMBLY KIT ADP, ACTUATOR BRACKET	1
4.1	NA	UPPER GUARD	1
4.2	NA	NYLOC NUT M8	2
4.3	NA	NUT M6 SW-14	3
4.4	NA	HEX HEAD SCREW M6 X 20	1
4.5	NA	CABLE STRAP L=99 MM (SHORT), BLACK	1
4.6	NA	SOCKET HEAD CAP SCREW M8 X 40	2
4.7	NA	SOCKET HEAD CAP SCREW M6 X 16	1
4.8	NA	SOCKET HEAD CAP SCREW M6 X 12	1
4.9	NA	ACTUATOR HOLDING PLATE	1
4.10	NA	LOWER GUARD	1

Legrests, sideways mounted - Assembly kit ADP, hoist motor

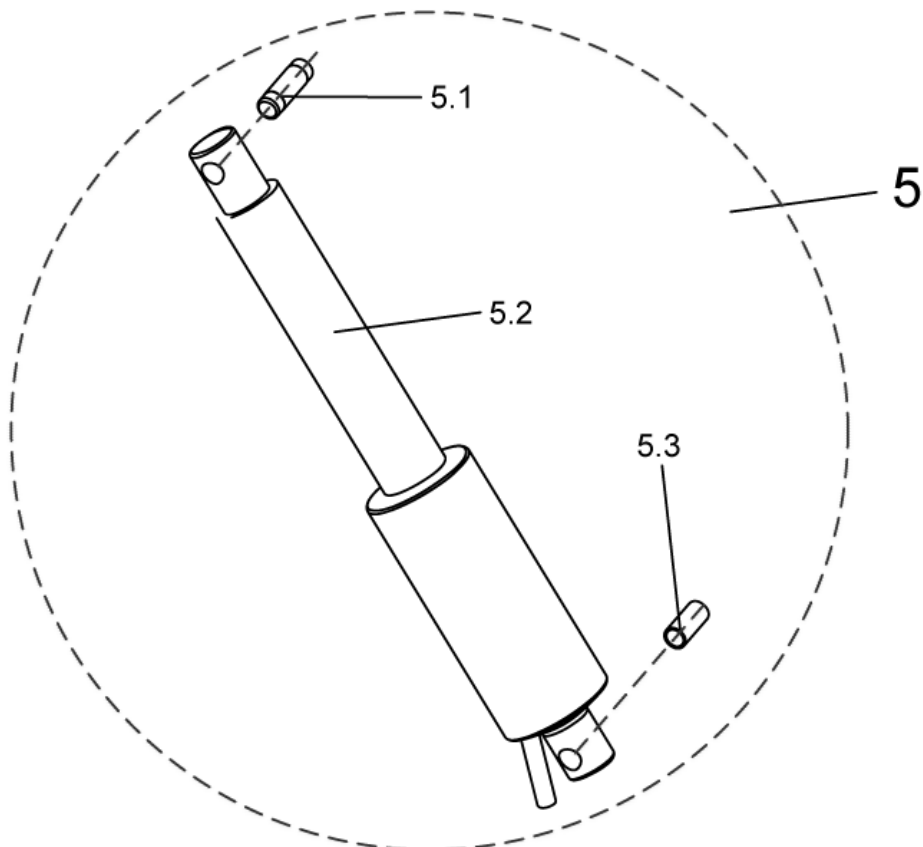
1449408_2.dwg




item	ordercode	description	quantity
5	1450606	ASSEMBLY KIT ADP, HOIST MOTOR	1
5.1	Z174137.99	PIVOT BUSH	1
5.2	SPZ151006	ACTUATOR ASSY	1
5.3	4000665-0002	SPACER BUSH 10X1X22,5	1

Legrests, sideways mounted - Lower Legrests Detailed

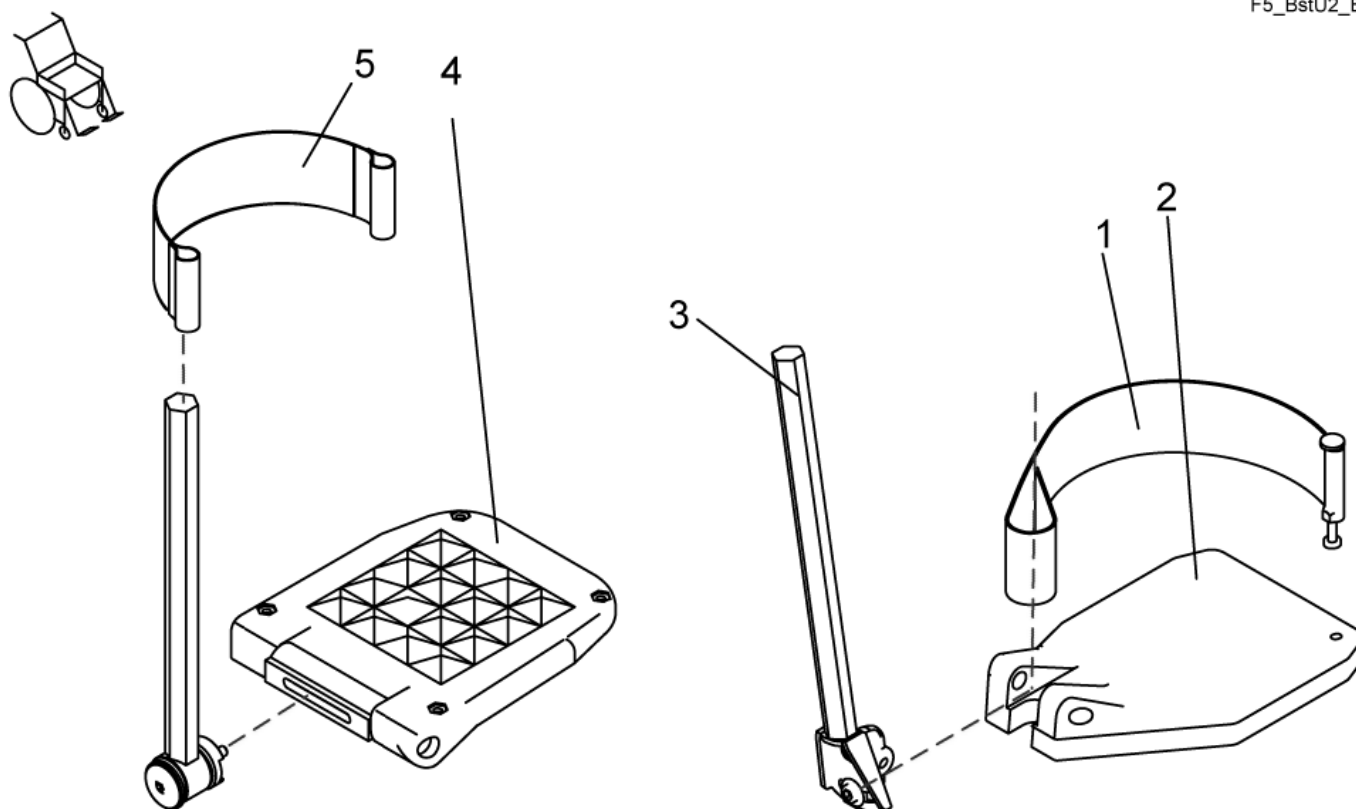
F5_BstU2_B.dwg



item	ordercode	description	quantity
1	SP1423202	HEEL STRAP CPL.	2
2	Z173217	FOOTPLATE LE; BLACK	1
2	Z173218	FOOTPLATE RH; BLACK	1
3	SP1423204	FOOT PLATE TUBE, ANGLE ADJUSTMENT LH CPL.	1
3	SP1423203	FOOT PLATE TUBE, ANGLE ADJUSTMENT RH CPL.	1
4	5106376	FOOTPLATE ADAPT / SPIREA ANGLE ADJ.	2
5	5321579	HEEL STRAP CPL.	2

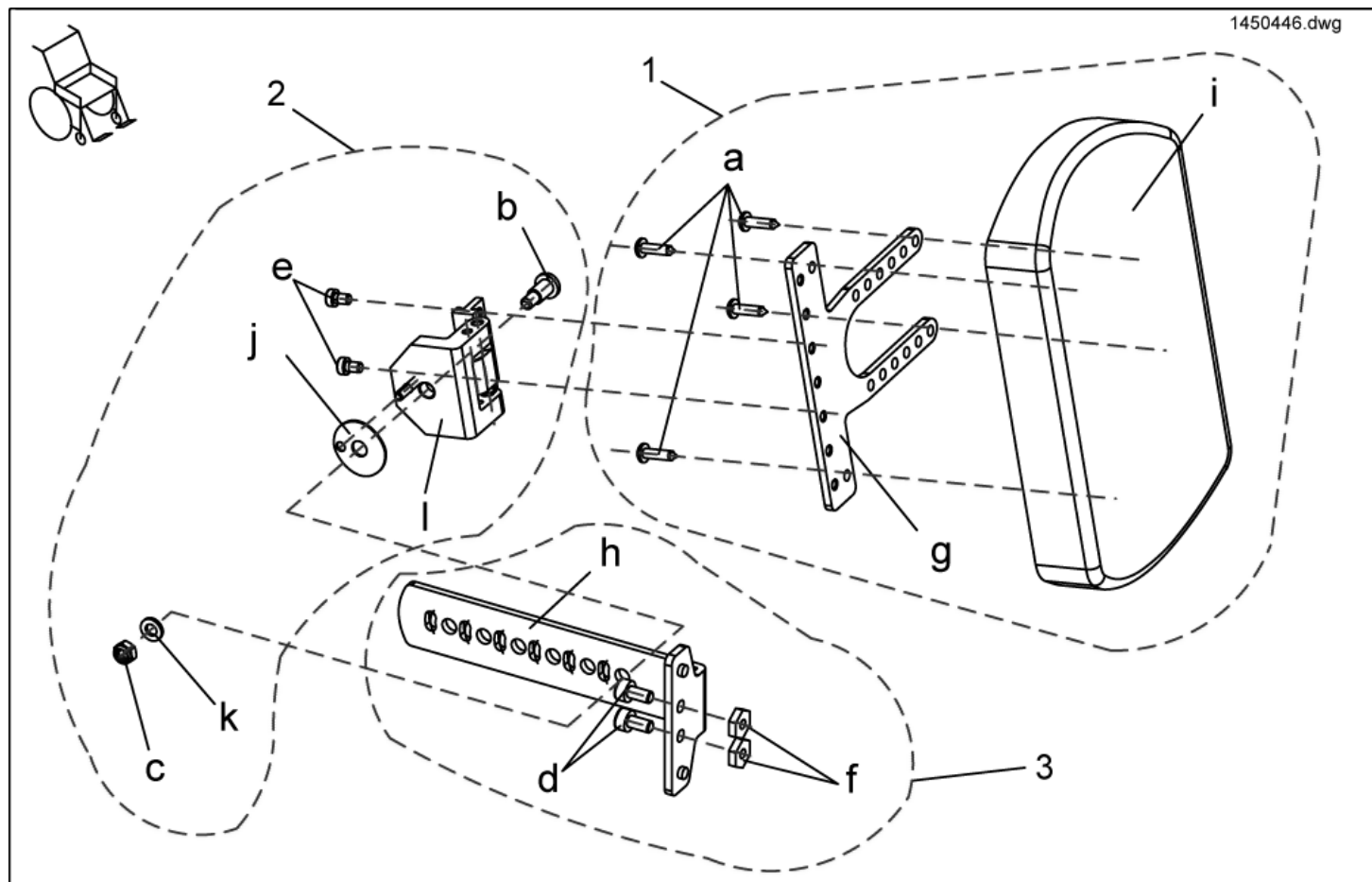


item	ordercode	description	quantity
-	SP1450446	CALF PLATE HOLDER RIGHT TURNABLE, COMPLETELY	1
1	SP1449021	ASSEMBLY KIT, CALF PLATE HOLDER	1
2	SP1449022	ASSEMBLY KIT, CALF PLATE HINGE	1
3	SP1449023	ASSEMBLY KIT, CALF PLATE	1
a	NA	DOMED CROSSHEAD SELF TAPPING SCREW 4,8 X 19	4
b	NA	SCREW M6 X 12	1
c	NA	NYLOC NUT M6	1
d	NA	SOCKET HEAD CAP SCREW M6 X 12	2
e	NA	SCREWS DIN 6912-M5x10	2
f	NA	NUT M6 SW-14	2
g	NA	ADAPTER SHEET METAL CALF PLATE TURNABLE / GALVANIZES	1
h	NA	CALF PLATE HOLDER TURNABLE / GALVANIZES	1
i	6301333	CALF PLATE	1
j	NA	SLIP WASHER FOR TURNABLE CALF PLATE	1
k	NA	WASHER 6,4 X 12,5 X 1,6	1
l	NA	CALF PLATE HINGE, RIGHT	1

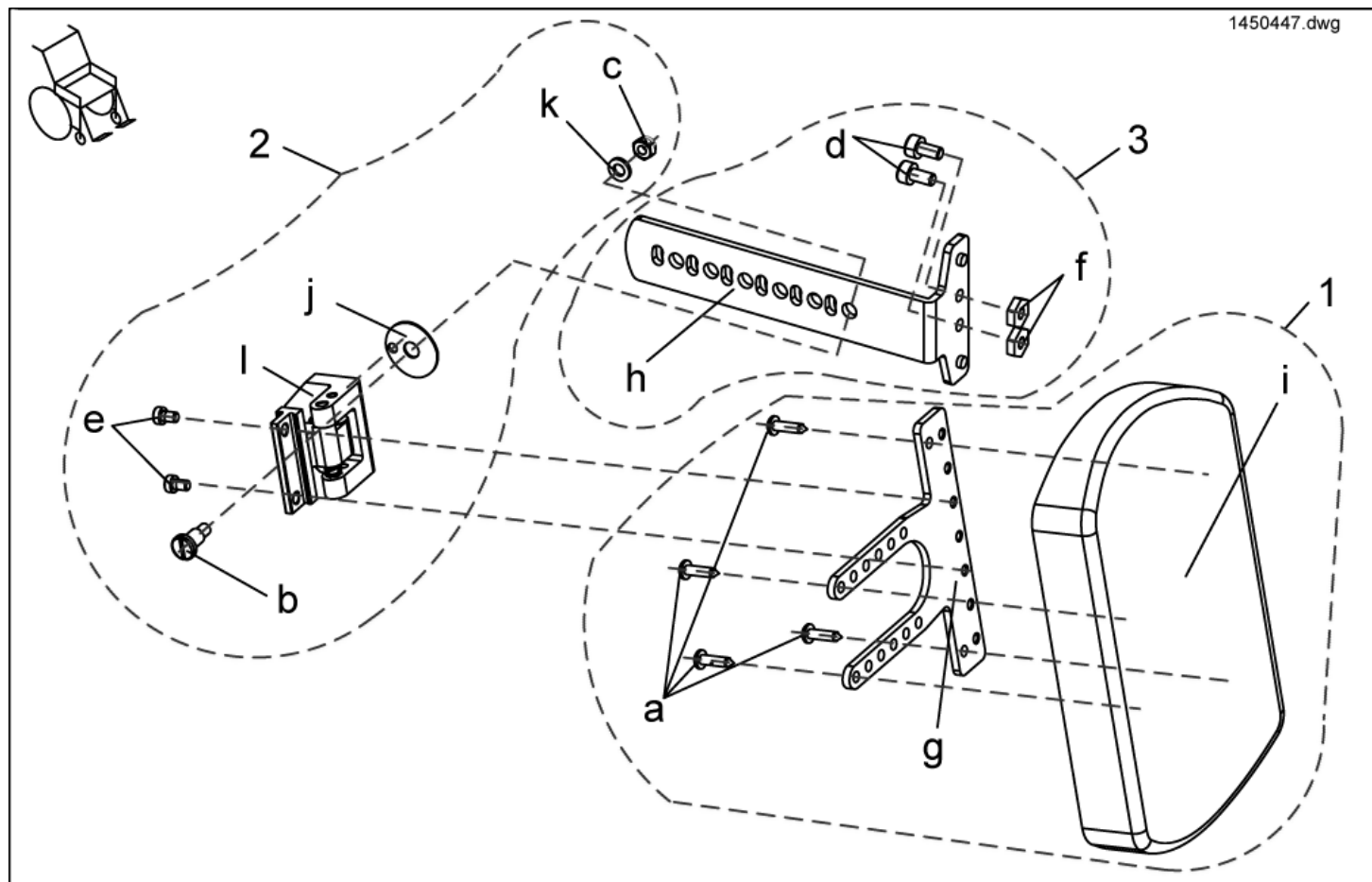


item	ordercode	description	quantity
-	SP1450447	CALF PLATE HOLDER LEFT TURNABLE, COMPLETELY	1
1	SP1449021	ASSEMBLY KIT, CALF PLATE HOLDER	1
2	SP1449022	ASSEMBLY KIT, CALF PLATE HINGE	1
3	SP1449023	ASSEMBLY KIT, CALF PLATE	1
a	NA	DOMED CROSSHEAD SELF TAPPING SCREW 4,8 X 19	4
b	NA	SCREW M6 X 12	1
c	NA	NYLOC NUT M6	1
d	NA	SOCKET HEAD CAP SCREW M6 X 12	2
e	NA	SCREWS DIN 6912-M5x10	2
f	NA	NUT M6 SW-14	2
g	NA	ADAPTER SHEET METAL CALF PLATE TURNABLE / GALVANIZES	1
h	NA	CALF PLATE HOLDER TURNABLE / GALVANIZES	1
i	6301333	CALF PLATE	1
j	NA	SLIP WASHER FOR TURNABLE CALF PLATE	1
k	NA	WASHER 6,4 X 12,5 X 1,6	1
l	NA	CALF PLATE HINGE, LEFT	1

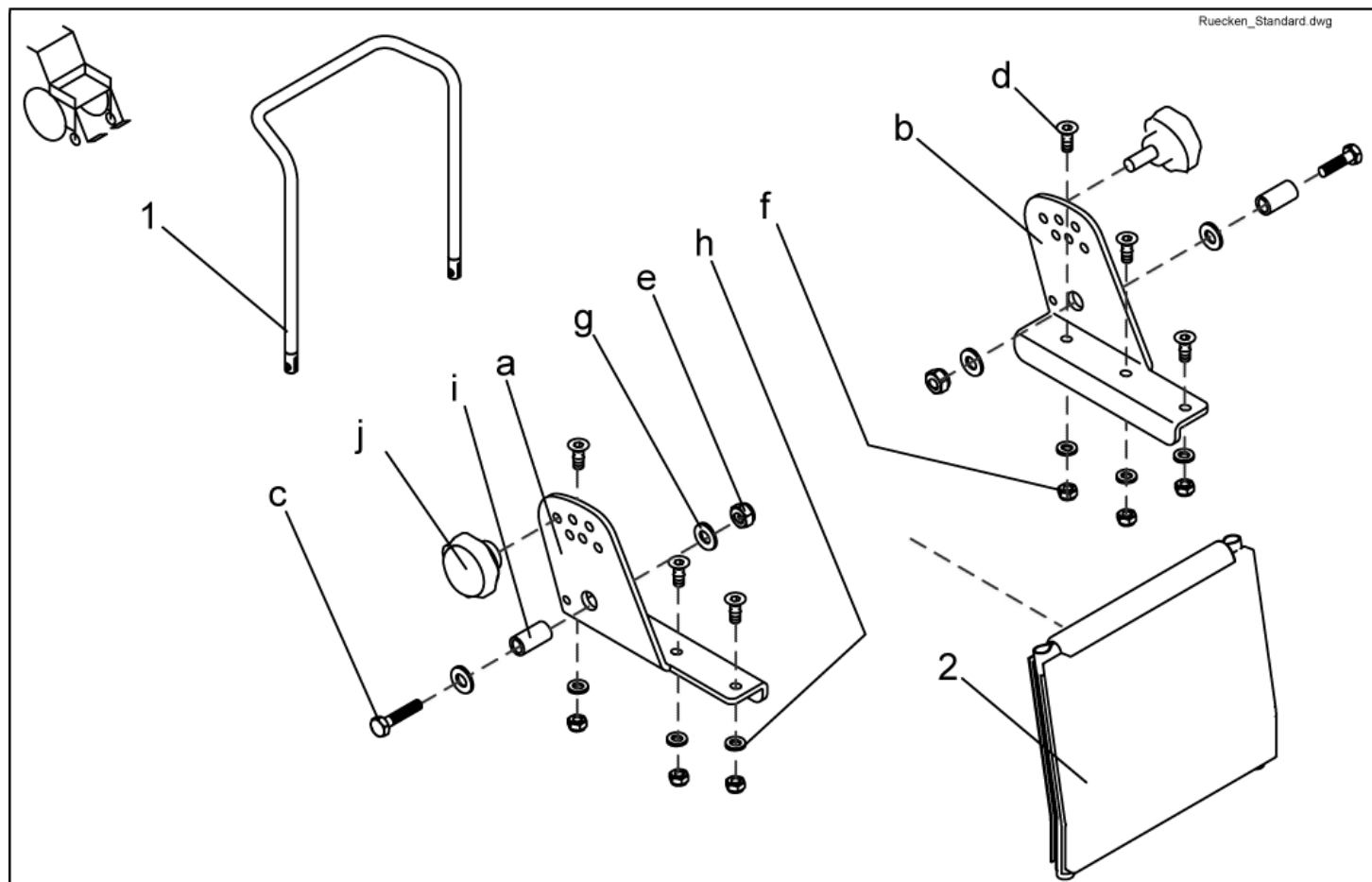Back Unit - Backrest frame Standard



item	ordercode	description	quantity
1	1419996-9005	BACK TUBE 480 MM	1
1	1420465-9005	BACK TUBE 540 MM	1
2	1422062	BACK STRAP 480 MM	1
2	1422063	BACK STRAP 540 MM	1
a,b,c,d,e,f,g,h,i,j	SP1423208	STD. BACKREST ASSEMBLY KIT CPL.	1
a	NA	BACKREST ADJUSTMENT RH	1
b	NA	BACKREST ADJUSTMENT LH	1
c	NA	HEX HEAD SCREW M8 X 35	2
d	NA	COUNTERSUNK SOCKET HEAD SCREW M6 X 16	6
e	NA	NYLOC NUT M8	2
f	NA	DOMED NYLOC NUT M6, SELF-LOCKING	6
g	NA	WASHER 8,4 X 17 X 1,6	4
h	NA	WASHER 6,4 X 12,5 X 1,6	6
i	NA	BUSH RD. 12 X 1.8 X 23	2
j	SP1420444	STAR GRIP M8 CPL.	2


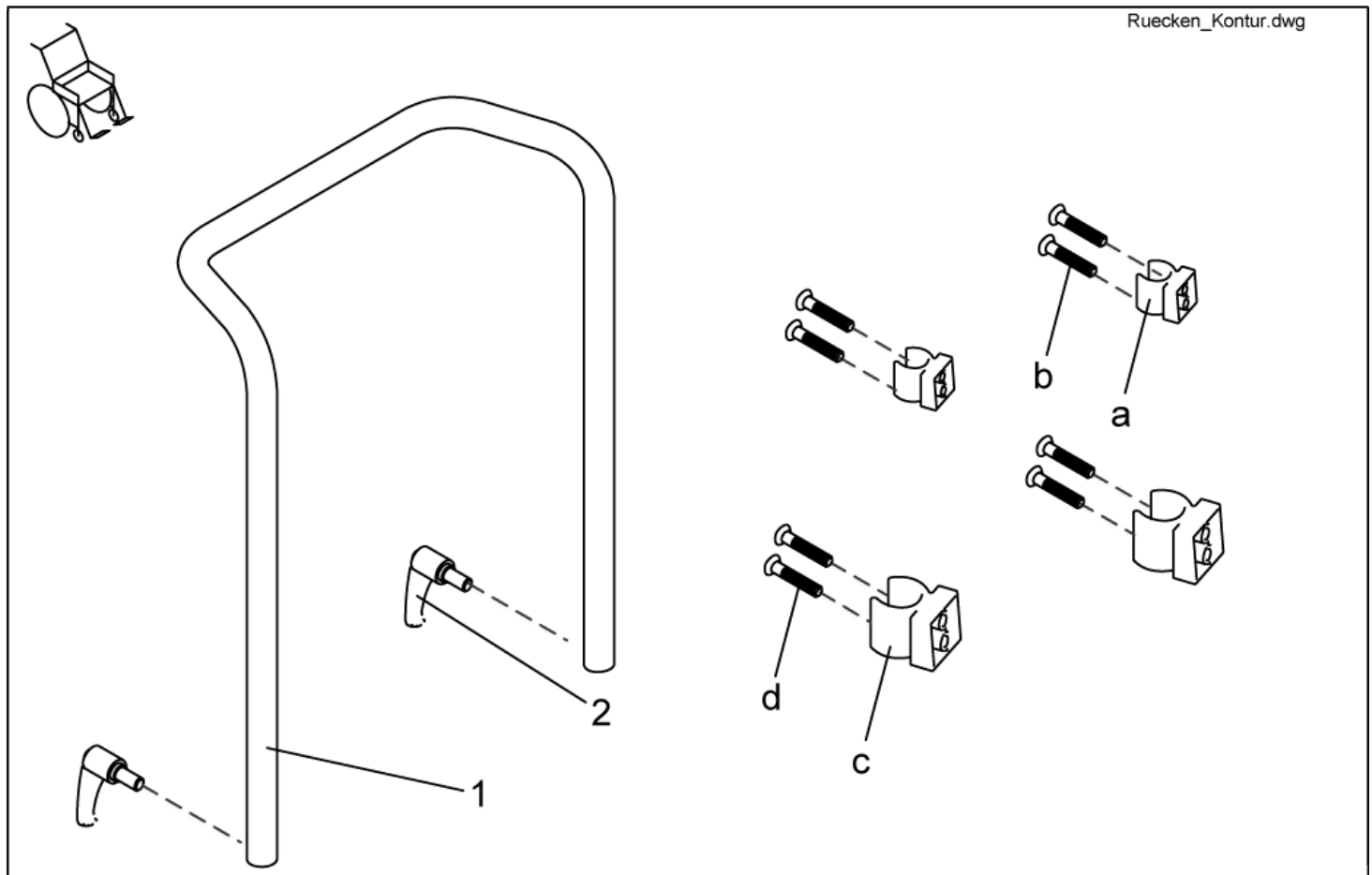
Back Unit - Backrest frame CONTOUR



item	ordercode	description	quantity
1	1420478-9005	BACK TUBE CONTOUR	1
2	L31996	LEVER M10 X 20	2
a,b,c,d	SP1423216	PADDED BACKREST UPHOLSTERY ASSEMBLY KIT	1
a	NA	CLAMP SHORT	2
b	NA	SCREW 5 X 30	4
c	NA	CLAMP LONG	2
d	NA	SCREW 5 X 45, DIN 7505	4


Back Unit - Backrest frame FLEX

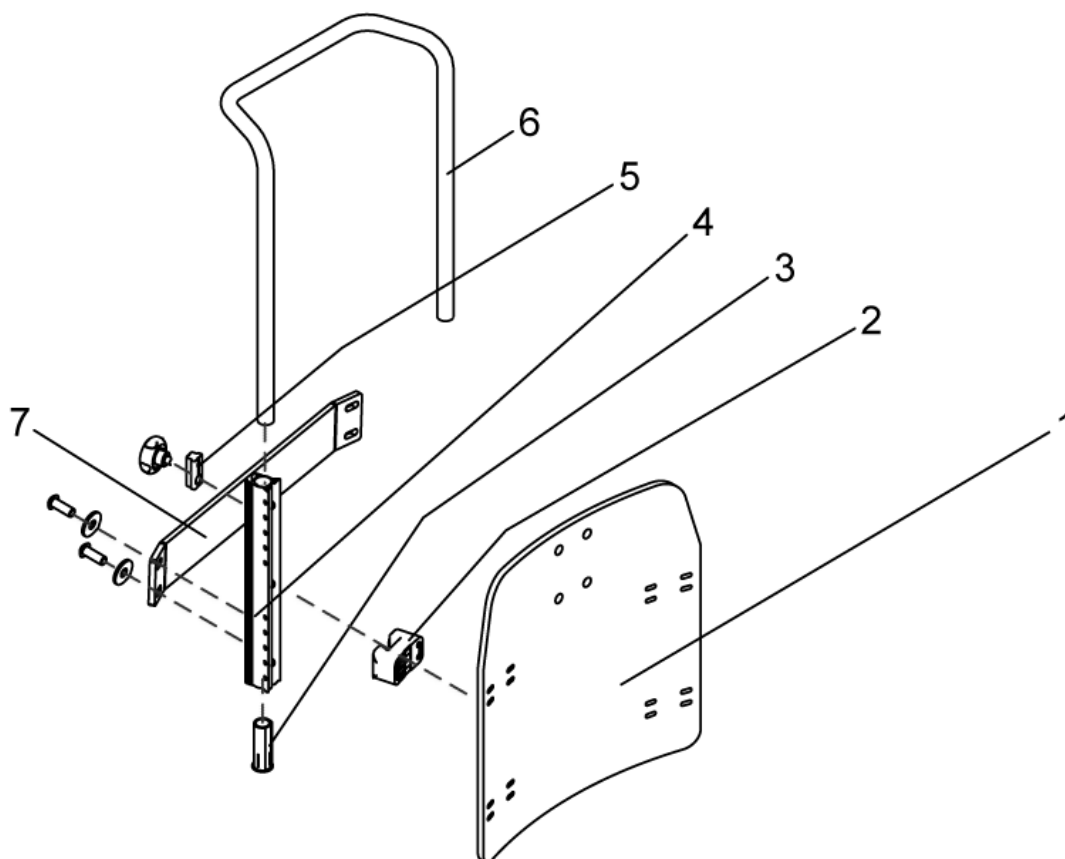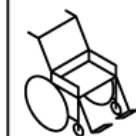Ruecken_Flex.dwg



item	ordercode	description	quantity
-	SP1422126	BACKREST FLEX SB 39 WITHOUT UPHOLSTERY CPL.	1
-	SP1422124	BACKREST FLEX SB 43 WITHOUT UPHOLSTERY CPL.	1
-	SP1422125	BACKREST FLEX SB 48 WITHOUT UPHOLSTERY CPL.	1
1	SP1423217	BACK PLATE SB 39 CPL.	1
1	SP1423218	BACK PLATE SB 43 CPL.	1
1	SP1423219	BACK PLATE SB 48 CPL.	1
2	SP1422117	SPACER BLOCK, 4 PIECES	1
3	SP1423220	GUIDE BUSH CPL. (PAIR)	1
4	5252204	BACKREST BAR	2
5	5253088	FIXING BLOCK	4
6	1425840-9005	PUSH HANDLE SB 39	1
6	1425839-9005	PUSH HANDLE SB 44/48	1
7	SP1488810	STABILIZATION SHEET METAL COMPLETELY; SEAT WIDTH 43/48CM	1
7	SP1488804	STABILIZATION SHEET METAL COMPLETELY; SEAT WIDTH 39CM	1

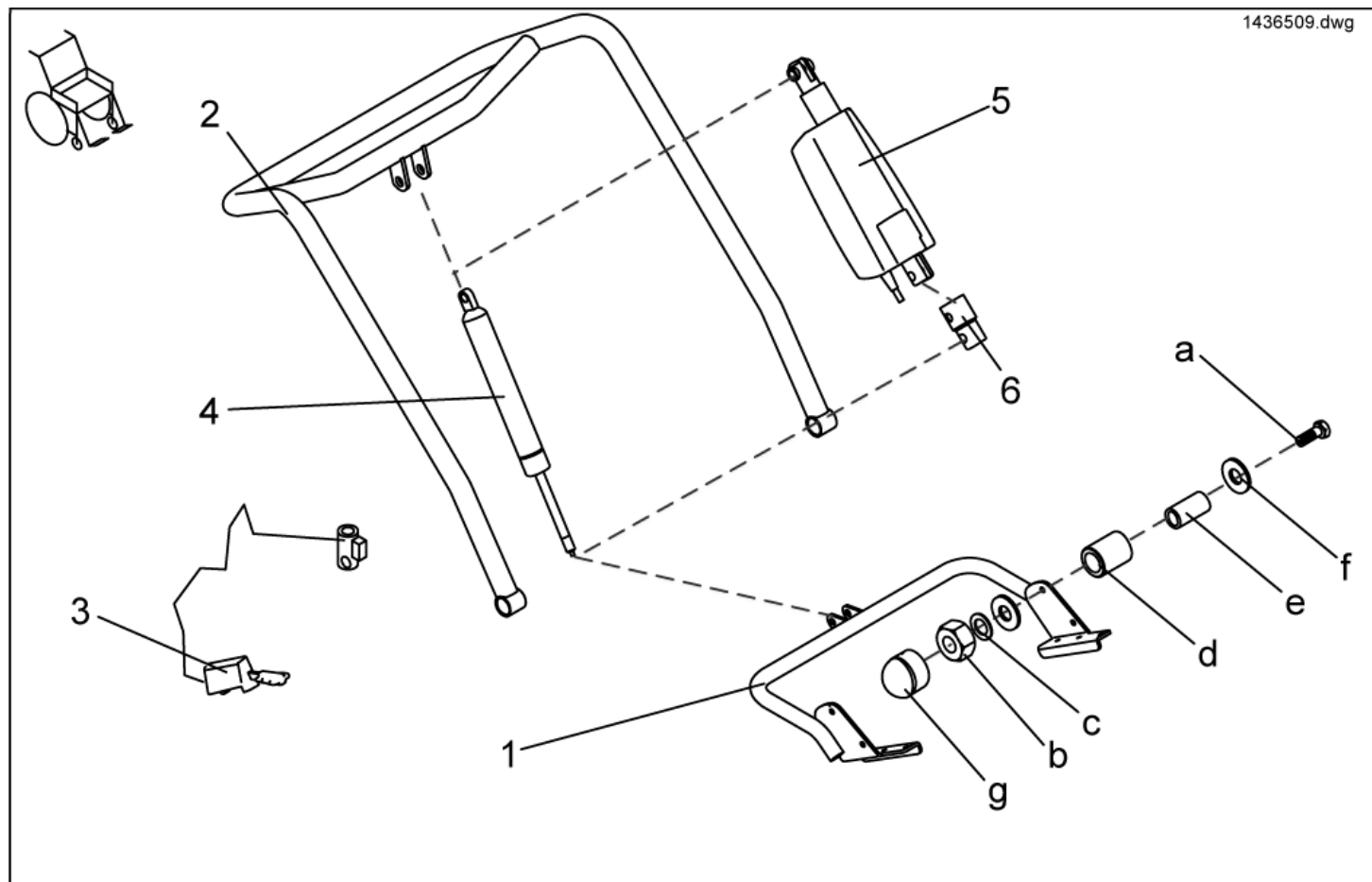
Back Unit - Backrest adjustment STANDARD



item	ordercode	description	quantity
-	SP1436509	BACKREST ADJUSTMENT PNEUMATIC FOR STANDARD SEAT COMPLETE	1
-	SP1436507	BACKREST ADJUSTMENT ELECTRICAL FOR STANDARD SEAT COMPLETE	1
-	SP1444991	BACK ADJUSTMENT STANDARD, BACK HEIGHT 54CM, PNEUMATICALLY, COMPLETELY WITH BACK BELT	1
-	SP1443270	BACK ADJUSTMENT STANDARD, BACK HEIGHT 54CM, ELECTRICALLY, COMPLETELY WITH BACK BELT	1
1	1440394-9005	LOWER RECEIVING BRACKET	1
2	1440395-9005	BACKTUBE	1
3	N368003	RELEASE LEVER "STABILUS"	1
4	N368007	BACK ADJUSTER PNEUMATIC CPL.	1
5	1418016	BACK ADJUSTER ELECTRIC CPL.	1
6	N368001.89	EXTENSION PART FOR ACTUATOR	1
a,b,c,d,e,f,g	F367005	FIXING KIT FOR SEAT MOUNTING	1
a	NA	AXLE BOLT M8 X 45	2
b	NA	NYLOC NUT M8	2
c	NA	WASHER 8,4 X 17 X 1,6	2
d	NA	BUSH RD. 18 X 3 X 22	4
e	NA	BUSH RD. 12 X 1.8 X 23	2
f	NA	WASHER 8,4 X 25 X 2 DIN 9021	2
g	NA	NUT CAP M8	2


Back Unit - Backrest adjustment FLEX/CONTUR man.

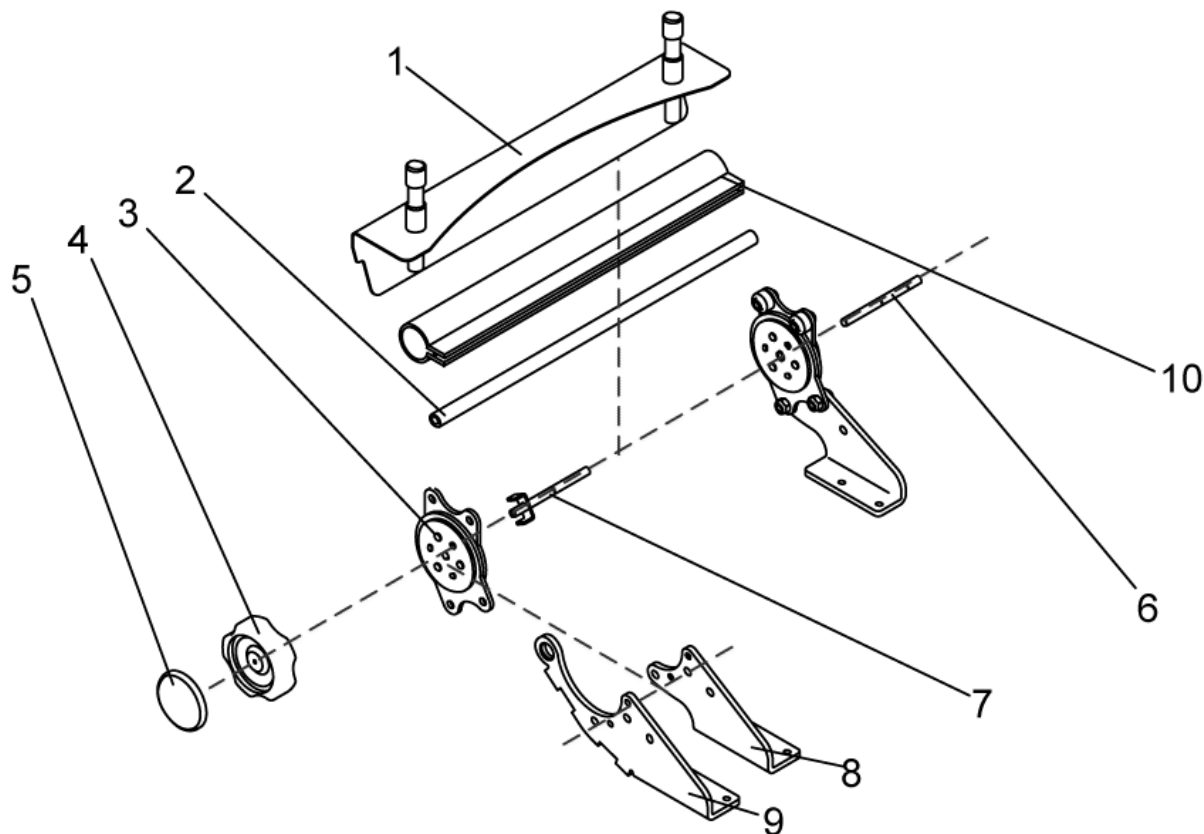SP142212X.dwg



item	ordercode	description	quantity
-	SP1422122	MANUAL BACKREST ADJUSTMENT F. FLEX SB 39 CPL.	1
-	SP1422066	MANUAL BACKREST ADJUSTMENT SB 43 CPL.	1
-	SP1422067	MANUAL BACKREST ADJUSTMENT SB 48 CPL.	1
-	SP1425857	Manual BACK ADJUSTER, SEAT WIDTH 39CM, FOR SLIDING ARMRESTS, cpl.	1
-	1442942	Manual BACK ADJUSTER, SEAT WIDTH 43cm, FOR SLIDING ARMRESTS, cpl.	1
-	SP1425854	Manual BACK ADJUSTER, SEAT WIDTH 48cm, FOR SLIDING ARMRESTS, cpl.	1
-	SP1423210	BACKREST ADJUSTMENT ASSEMBLY KIT, CPL.	1
1	1422119-9005	BACKREST RECEPTACLE 390 MM	1
1	1419992-9005	BACKREST RECEPTACLE 430 MM	1
1	1422112-9005	BACKREST RECEPTACLE 480 MM	1
2	1420489	CONNECTING TUBE 380 MM	1
2	1420490	CONNECTING TUBE 430 MM	1
2	1420491	CONNECTING TUBE 480 MM	1
3	SP1450234	SEAT BRACE (1 PAIR)	1
4	1420483	HAND WHEEL	1
5	1421696	PERFORATED REFLECTOR; YELLOW	2
6	1420488-0002	SUPPORT SHAFT	1
7	1421713-0002	HAND WHEEL SHAFT	1
8	1420477-9005	BRACE RECEPTACLE manual/electrical LH	1
8	1420476-9005	BRACE RECEPTACLE manual RH	1
9	1429564-9005	FITTING ATTACHMENT LH F. SLIDING ARMREST	1
9	1429563-9005	FITTING ATTACHMENT RH F. MANUAL ADJUSTMENT	1
10	1442577	PROTECTIVE COVERING 380MM	1
10	1442578	PROTECTIVE COVERING 430MM	1
10	1442577	PROTECTIVE COVERING 480MM	1

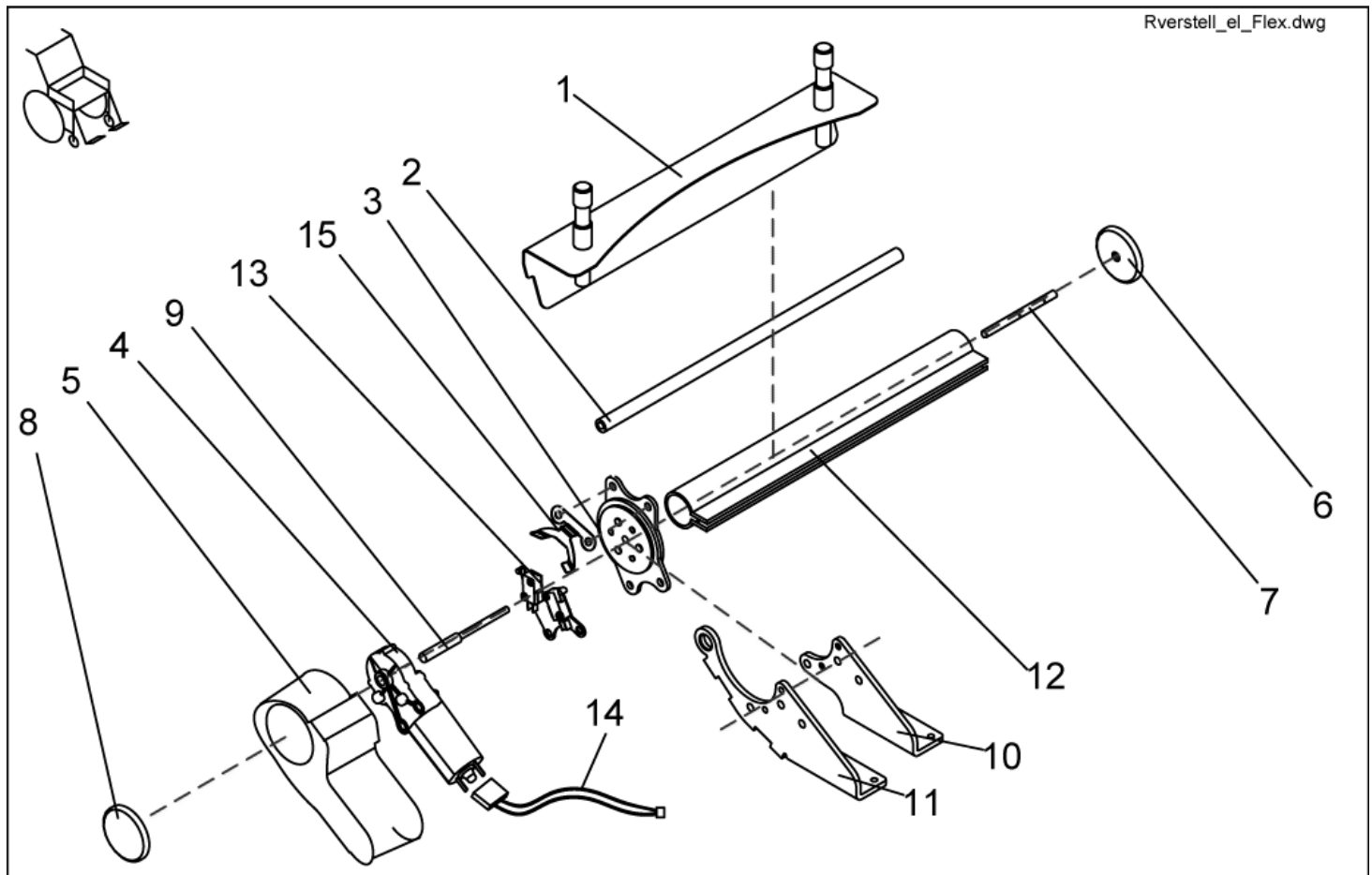Back Unit - Backrest adjustment FLEX/CONTUR el.




item	ordercode	description	quantity
-	SP1440407	ELECTRICAL BACK ADJUSTER, SEAT WIDTH 39CM, CPL.	1
-	SP1440405	ELECTRICAL BACK ADJUSTER, SEAT WIDTH 43CM, CPL.	1
-	SP1440406	ELECTRICAL BACK ADJUSTER, SEAT WIDTH 48CM, CPL.	1
-	SP1440410	ELECTRICAL BACK ADJUSTER, SEAT WIDTH 39CM, FOR SLIDING ARMRESTS, CPL.	1
-	SP1440408	ELECTRICAL BACK ADJUSTER, SEAT WIDTH 43CM, FOR SLIDING ARMRESTS, CPL.	1
-	SP1440409	ELECTRICAL BACK ADJUSTER, SEAT WIDTH 48CM, FOR SLIDING ARMRESTS, CPL.	1
-	SP1423210	BACKREST ADJUSTMENT ASSEMBLY KIT, CPL.	1
1	1422119-9005	BACKREST RECEPTACLE 390 MM	1
1	1419992-9005	BACKREST RECEPTACLE 430 MM	1
1	1422112-9005	BACKREST RECEPTACLE 480 MM	1
2	1420489	CONNECTING TUBE 380 MM	1
2	1420490	CONNECTING TUBE 430 MM	1
2	1420491	CONNECTING TUBE 480 MM	1
3	SP1423211	SEAT BRACE (1 PAIR)	1
4	1420475	GEAR MOTOR GMPG	1
5	1420998	MOTOR CAP	1
6	1421696	PERFORATED REFLECTOR; YELLOW	2
7	1420488-0002	SUPPORT SHAFT	1
8	1420482	REFLECTOR	1
9	1420487-0002	MOTOR SHAFT	1
10	1420477-9005	BRACE RECEPTACLE manual/electrical LH	1
10	1421708-9005	BRACE RECEPTACLE electrical, RH	1
11	1429564-9005	FITTING ATTACHMENT LH F. SLIDING ARMREST	1
11	1429563-9005	FITTING ATTACHMENT RH F. MANUAL ADJUSTMENT	1
12	1442577	PROTECTIVE COVERING 380MM	1
12	1442578	PROTECTIVE COVERING 430MM	1
12	1442579	PROTECTIVE COVERING 480MM	1

item	ordercode	description	quantity
13	SP1440711	TWO MICRO SWITCHES WITH ASSEMBLY SHEET METAL, PRE-MOUNTS	1
14	1440726	ACTUATORCABLE RECLINER	1
15	1440456-0003	SWITCHING OFF SHEET METAL BACK	1

Back Unit - Backrest adjustment ASSEMBLY KIT, cpl.

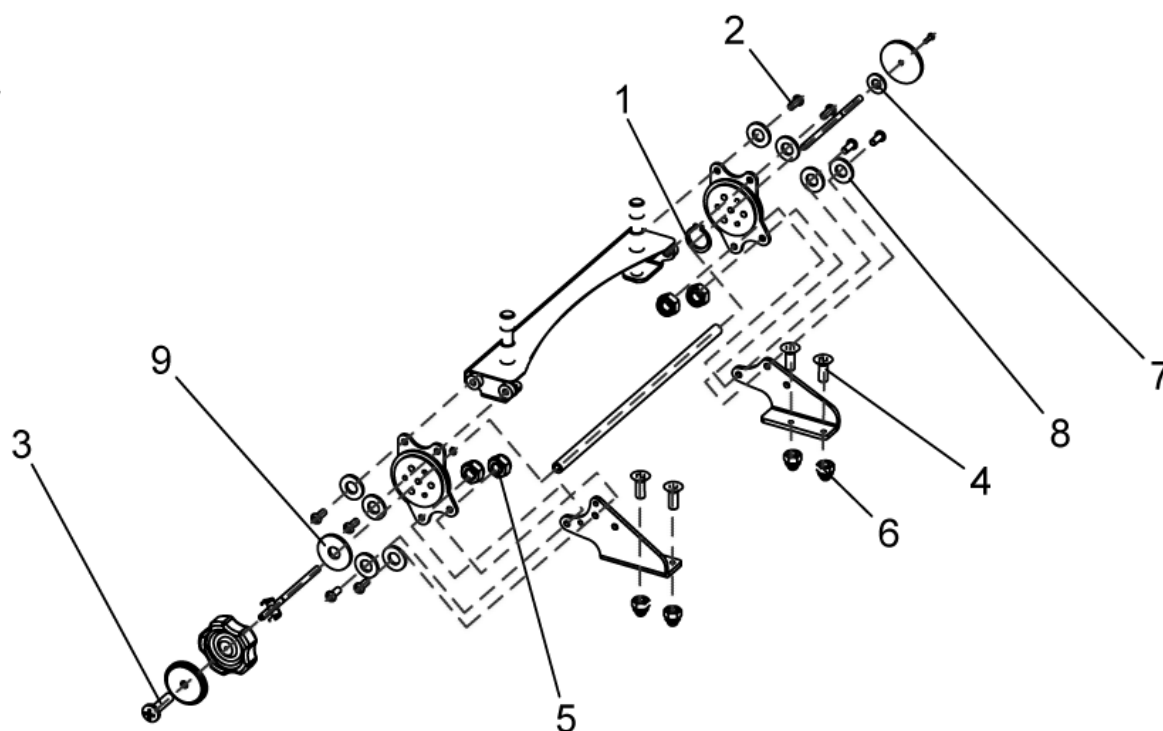
SP1423210.dwg



item	ordercode	description	quantity
-	SP1423210	BACKREST ADJUSTMENT ASSEMBLY KIT, CPL.	1
1	NA	RETAINING RING	2
2	NA	SCREW M8 X 20	8
3	NA	CROSS HEAD SCREW M4 X 16	2
4	NA	COUNTERSUNK SOCKET HEAD SCREW M6 X 16	4
5	NA	NYLOC NUT M8	4
6	NA	DOMED NYLOC NUT M6, SELF-LOCKING	4
7	NA	WASHER 4,3 X 9 X 0,8	1
8	NA	WASHER 8,4 X 17 X 1,6	11
9	NA	WASHER 8,4 X 25 X 2	1

Back Unit - Retaining belts

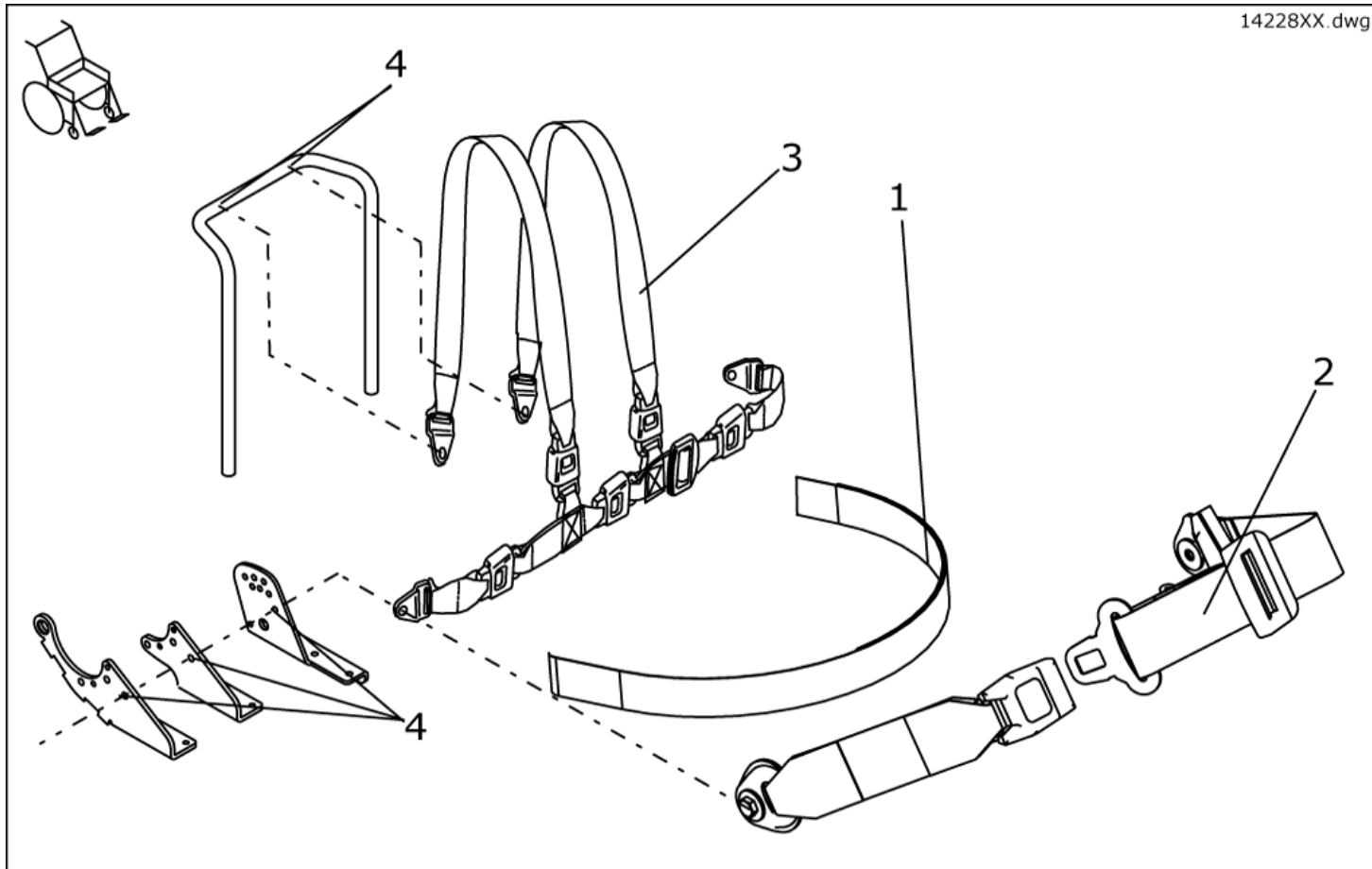
14228XX.dwg



item	ordercode	description	quantity
1	1422869	POSITIONING STRAP W. VELCRO CPL. FOR STANDARD-SEAT	1
1	1422870	POSITIONING STRAP W. VELCRO CPL. FOR FLEX/CONTOUR-SEAT	1
1	F367002	SAFETY BELT WITH VELCRO FASTNER F. RECARO	1
2	1422845	POSITIONING STRAP W. SNAP CPL. FOR STANDARD-SEAT	1
2	1422846	POSITIONING STRAP W. SNAP CPL. FOR FLEX/CONTOUR	1
2	Z163001A	SAFETY BELT WITH SNAP LOCK AND VELCRO FASTNER FOR RECARO	1
3	1422848	H-STRAP STD./CONTOUR	1
3	1422847	H-STRAP FLEX	1
4	NA	SCREW-DOWN POINTS FOR RETAINING BELTS	2, 4

Back Unit - Head rests

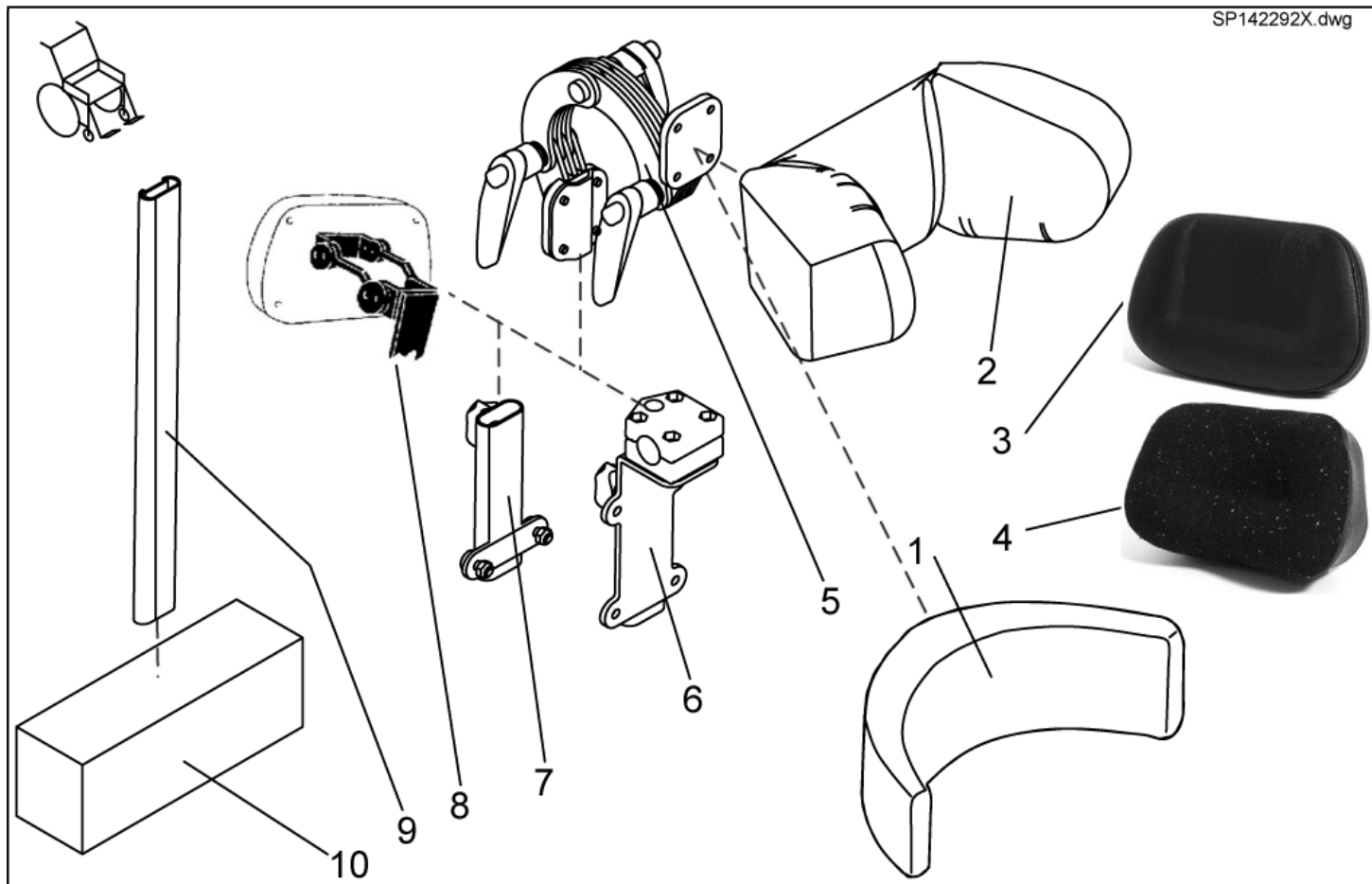SP142292X.dwg











item	ordercode	description	quantity
1,5	SP1422925	HEADREST CONTOUR BLACK CPL.	1
2,5	SP1423163	HEADREST FLEX GREY CPL.	1
2,5	SP1428092	HEADREST FLEX, DARTEX GREY, COMPLETE	1
2,5	SP1422926	HEADREST FLEX BLUE CPL.	1
2,5	SP1440285	HEADREST FLEX, BLACK COMPLETE	1
1	1422865	HEADREST PAD STD./CONTOUR BLACK	1
2	5252230-TR04	HEADREST PAD FLEX GREY	1
2	1422544	HEADREST PAD FLEX BLUE	1
2	1440285	HEADREST PAD FLEX BLACK	1
3	1419640	HEAD REST PAD ARNE JENSEN STANDARD CHILDREN	1
3	5313999	HEAD REST PAD ARNE JENSEN STANDARD ADULTS	1
4	1419646	HEAD REST PAD ARNE JENSEN TYPE 4 CHILDREN	1
4	1419643	HEAD REST PAD ARNE JENSEN TYPE 4 ADULTS	1
5	5255497	HEADREST HOLDER	1
6	SP1422914	HEADREST HOLDER F. STD. SEAT SYSTEM CPL.	1
7	SP1422547	HEADREST HOLDER F. CONTOUR/FLEX. SEAT SYSTEM CPL.	1
8	1419637	HEAD REST HOLDER ARNE JENSEN TYPE 3 CHILDREN	1
8	1419636	HEAD REST HOLDER ARNE JENSEN TYPE 3 ADULTS	1
9	1422921	HOLDER F. BACKREST EXTENSION	1
10	5253035-TR04	BACKREST EXTENSION SB 39; GREY	1
10	5253035-TR23	BACKREST EXTENSION REA 704 SB 39; DARTEX GREY	1
10	SP1422918	BACKREST EXTENSION FLEX SB 39; BLUE	1
10	SP1436567	BACKREST EXTENSION FLEX SB 39; BLACK	1
10	5253036-TR04	BACKREST EXTENSION SB 44; GREY	1
10	5253036-TR23	BACKREST EXTENSION REA 704 SB 43; DARTEX GREY	1
10	SP1422919	BACKREST EXTENSION FLEX SB 43; BLUE	1
10	SP1436568	BACKREST EXTENSION FLEX SB 43; BLACK	1

item	ordercode	description	quantity
10	5253037-TR04	BACKREST EXTENSION SB 49; GREY	1
10	5253037-TR23	BACKREST EXTENSION REA 704 SB 48; DARTEX GREY	1
10	SP1422920	BACKREST EXTENSION FLEX SB 48; BLUE	1
10	SP1440283	BACKREST EXTENSION FLEX SB 48; BLACK	1


ONLY FOR SEAT SYSTEMS WITHOUT HEADREST !

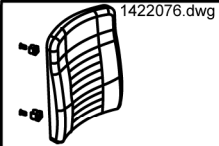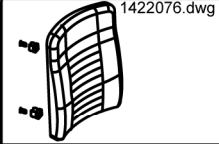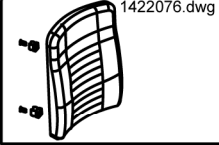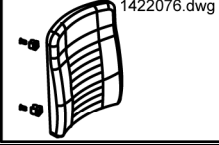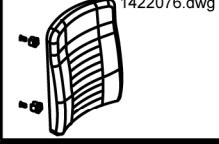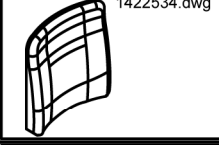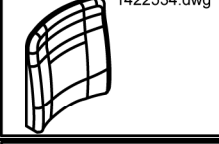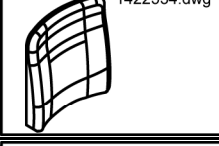Back Unit - Backrest Upholstery

Picture	Seat system	Seat width cm	color	ordercode
 1422076.dwg	Kontur	43	grey	SP1422075
 1422076.dwg	Kontur	43	black	SP1422076
 1422076.dwg	Kontur	48	grey	SP1422077
 1422076.dwg	Kontur	48	black	SP1422078
 1422076.dwg	Kontur Euro	-	black	SPF365037
 1422534.dwg	Flex	39	blue	1422534
 1422534.dwg	Flex	39	Vinyl grey	5252506-TR04
 1422534.dwg	Flex	39	Dartex grey	5252506-TR23
 1422534.dwg	Flex	39	black	1436564
 1422534.dwg	Flex	43	blue	1422535
 1422534.dwg	Flex	43	Vinyl grey	5252507-TR04

Picture	Seat system	Seat width cm	color	ordercode
 1422534.dwg	Flex	43	Dartex grey	5252507-TR23
 1422534.dwg	Flex	43	black	1436565
 1422534.dwg	Flex	48	blue	1422536
 1422534.dwg	Flex	48	Vinyl grey	5252508-TR04
 1422534.dwg	Flex	48	Dartex grey	5252508-TR23
 1422534.dwg	Flex	48	black	1436566
 1422537.dwg	Flex Vic Air	43	VicAir	1422537

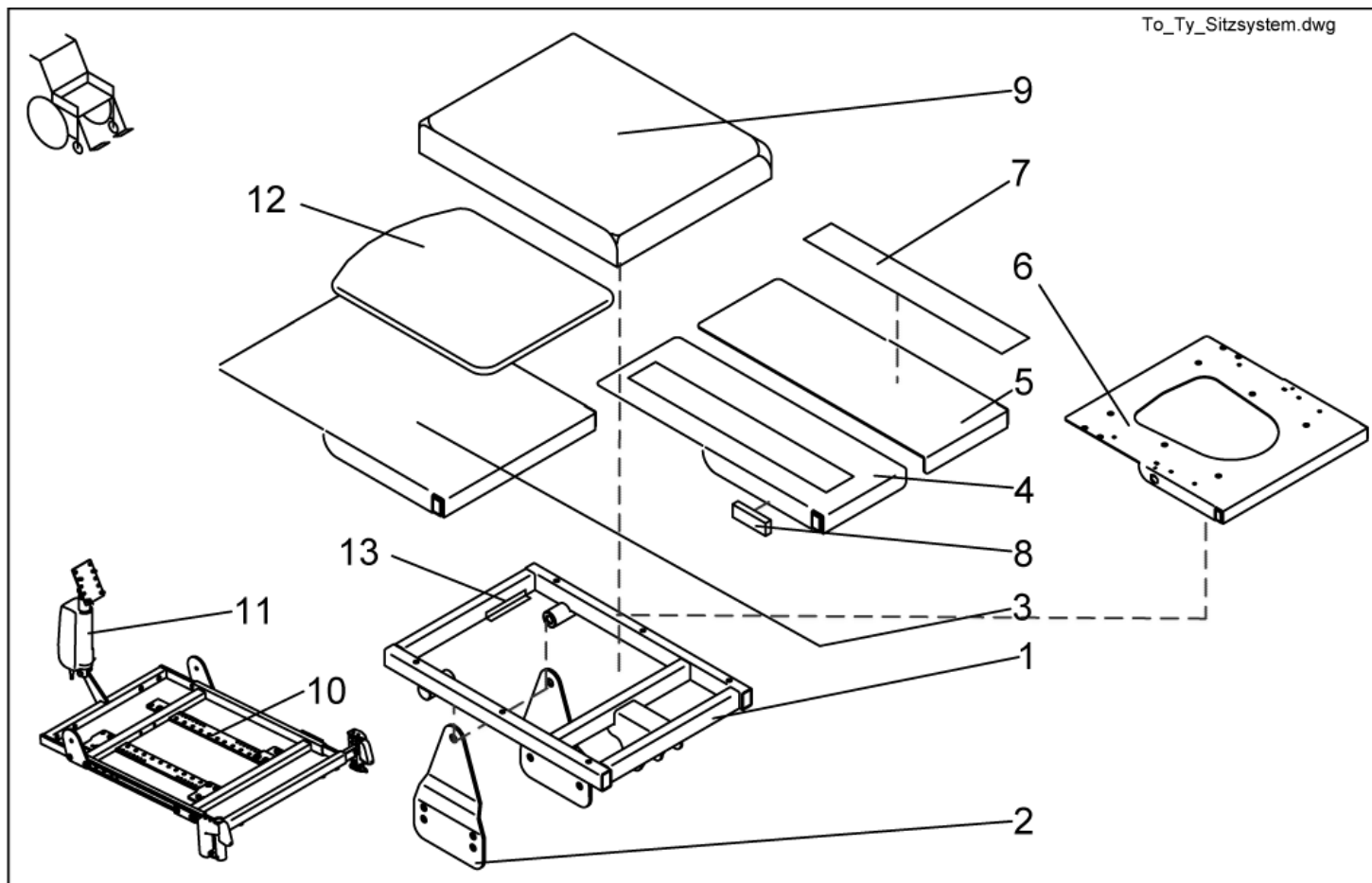Seat unit - Seat Support/ Plate




item	ordercode	description	quantity
1	SP1436455	SEAT SUPPORT STANDARD COMPLETE	1
-	1440623	MOUNTING KIT SEAT PLATE, USABLE FOR ALL VERSIONS	1
2	1442756	HOLDING PLATES FOR SEATFRAME STANDARD WITH MOUNTING KIT, SEATHEIGHT 45CM AND 47CM	1
3	SP1436514	SEAT PLATE (SINGLE PART) STANDARD COMPL.	1
3	1440402-9005	SEAT PLATE (ONE PIECE)	1
4+5	SP1436517	SPLIT SEAT SB 39 CPL.	1
4+5	SP1442832	SPLIT SEAT SB 39, FOR ELECTRICAL CENTRE MOUNTED LEGREST, COMPLETE	1
4+5	SP1436515	SPLIT SEAT SB 43 CPL.	1

item	ordercode	description	quantity
4+5	SP1436516	SPLIT SEAT SB 48 CPL.	1
4	1420457-9005	SEAT PLATE RH CPL.	1
4	1442813-9005	SEAT PLATE RH SB 39, FOR ELECTRICAL CENTRE MOUNTED LEGREST	1
5	1420458-9005	SEAT PLATE LH CPL.	1
5	1442814-9005	SEAT PLATE LH SB 39, FOR ELECTRICAL CENTRE MOUNTED LEGREST	1
6	SP1436518	SEAT PLATE RECARO CPL.	1
7	U21087	CUSHION FIXING STRIP - VELCRO FASTENER (HOOKED) 38 X 400	2
8	P31298	REFLECTOR, YELLOW	2
9		»»» see "Cushion"	1
10	SP1445570	seat frame OPTIMIST, seatwidth 38cm, with motor for backrest, for central mounted legrest, premounted	1
10	SP1436520	seat frame OPTIMIST, seatwidth 45cm, with motor for backrest, for central mounted legrest, premounted	1
10	SP1445571	seat frame OPTIMIST, seatwidth 50cm, with motor for backrest, for central mounted legrest, premounted	1
10	SP1445561	seat frame OPTIMIST, seatwidth 38cm, with motor for backrest, for sideways mounted legrest, premounted	1
10	SP1445028	seat frame OPTIMIST, seatwidth 48cm, with motor for backrest, for sideways mounted legrest, premounted	1
10	SP1445569	seat frame OPTIMIST, seatwidth 50cm, with motor for backrest, for sideways mounted legrest, premounted	1
11	1419417	MOTOR LA 28.1-100-24- 001	1
12	5314532	SEAT PLATE 410 X 520 MM	1

item	ordercode	description	quantity
12	5314533	SEAT PLATE 460 X 520 MM	1

item	ordercode	description	quantity
13	S182007.P40	ATTACHING BRACKET FOR SRB PLUG	1

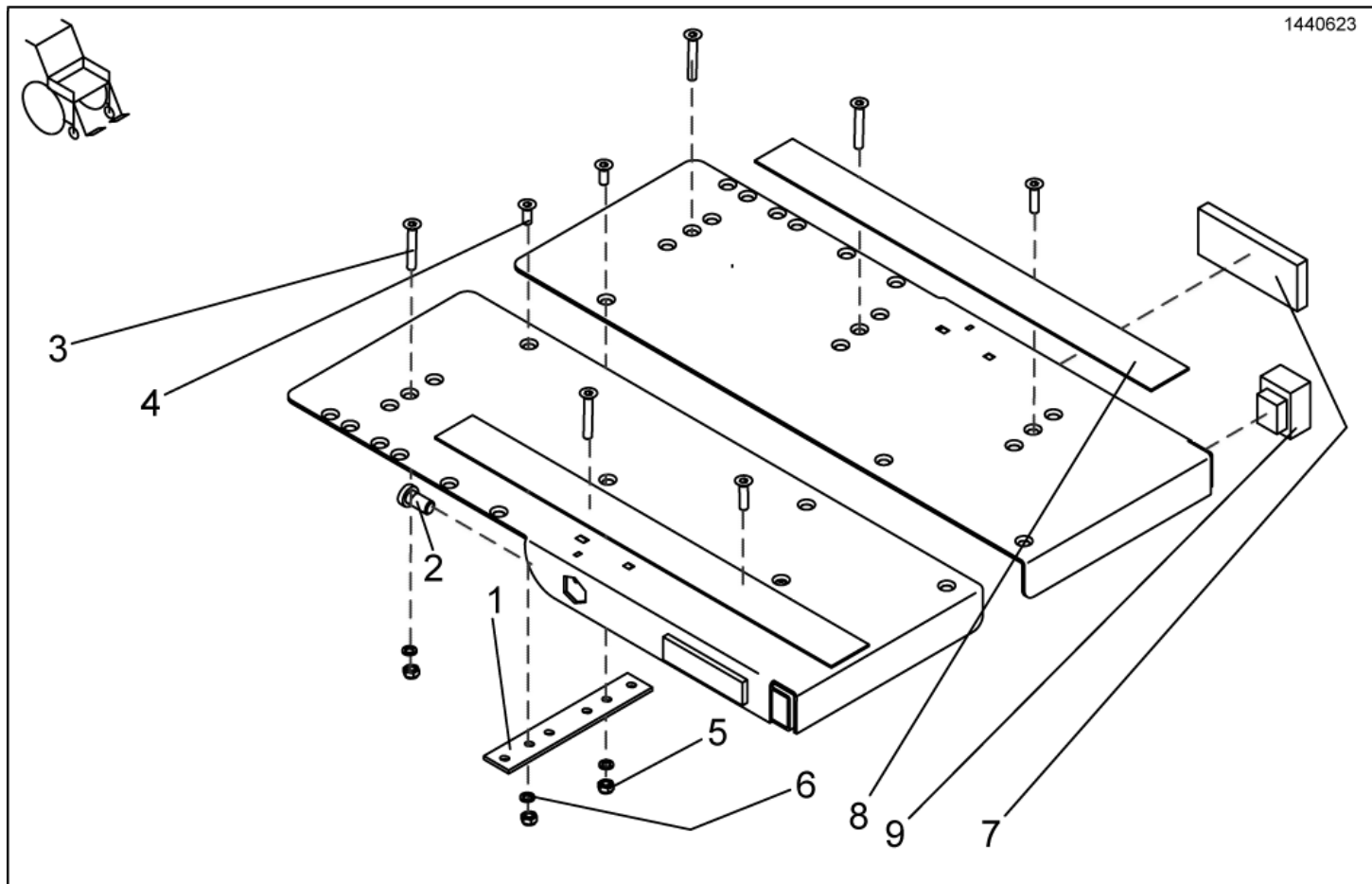
Seat unit - Monting kit seat plate



item	ordercode	description	quantity
-	1440623	MOUNTING KIT SEAT PLATE, USABLE FOR ALL VERSIONS	1
1	NA	REINFORCING PLATE	1
2	NA	BOLT M10 X 20	2
3	NA	COUNTERSUNK SOCKET HEAD SCREW M6 X 45 10.9	4
4	NA	COUNTERSUNK SOCKET HEAD SCREW M6 X 16	4
5	NA	NYLOC NUT M6	8
6	NA	WASHER 6,4 X 12,5 X 1,6	6
7	P31298	REFLECTOR, YELLOW	2
8	U21087	CUSHION FIXING STRIP - VELCRO FASTENER (HOOKED) 38 X 400	2
9	NA	TUBE END-PLUG SQ 25 MM; BLACK	2

Seat unit - RECARO-Seats

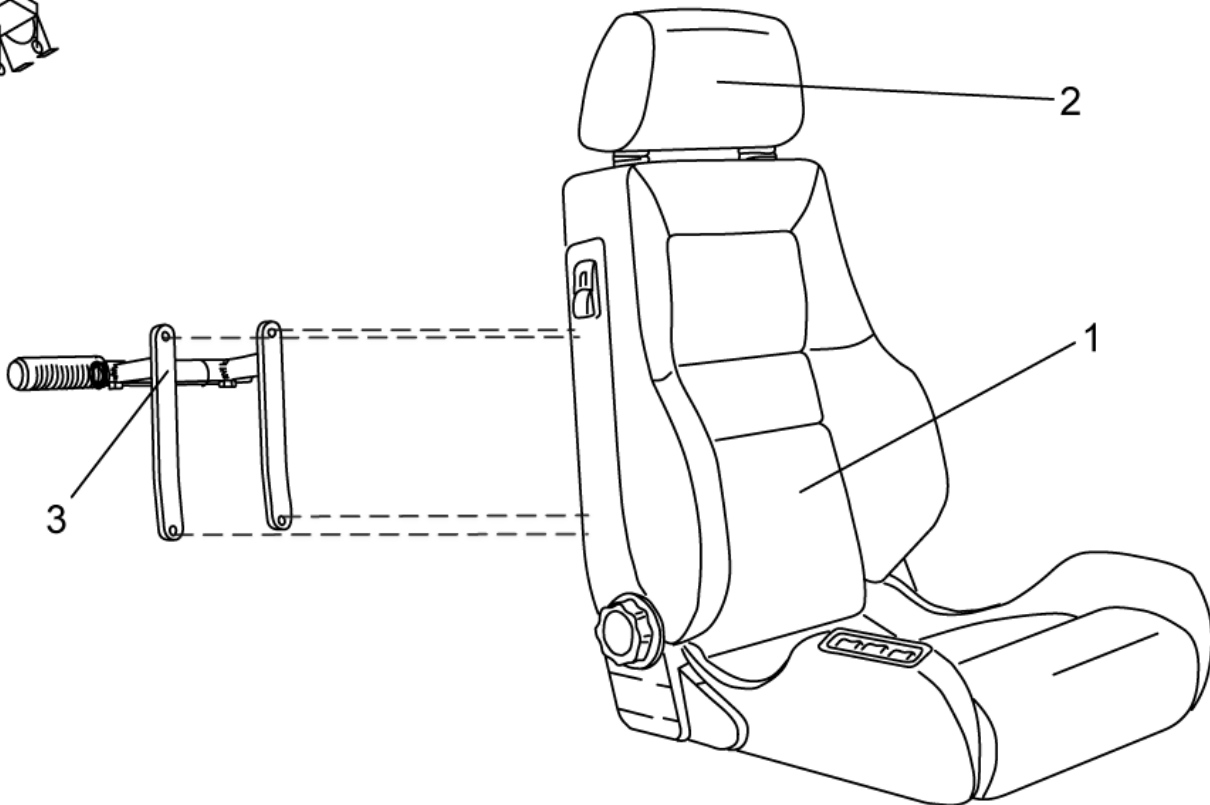
F3_RECARO-Sitz.dwg



item	ordercode	description	quantity
1	SP1422546	RECARO SEAT N-JOY WITH MOUNTING KIT	1
1	SP1420495	RECARO SEAT MOBILITY WITH MOUNTING KIT	1
-	SP1423173	RECARO SEAT MOUNTING KIT	1
-	F367002	SAFETY BELT WITH VELCRO FASTNER F. RECARO	1
-	Z163001A	SAFETY BELT WITH SNAP LOCK AND VELCRO FASTNER FOR RECARO	1
2	N89999	HEADREST - SPORT NARDO	1
3	SP1423177	HOLDER F. PUSH HANDLE RECARO CPL.	1









Seat unit - Holder for push handle RECARO cpl.



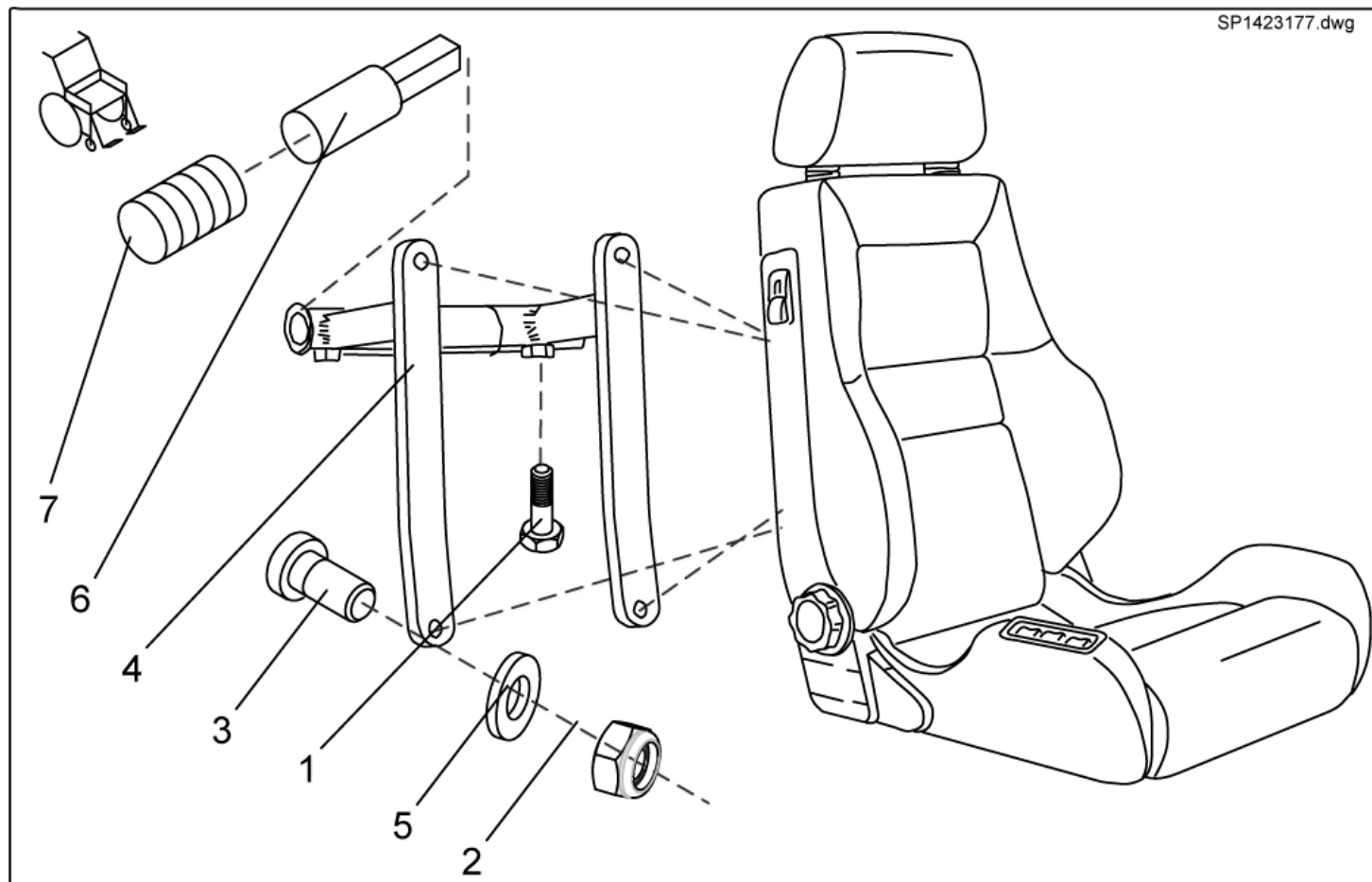
WE RECOMMEND RETROFITTING AT THE FACTORY !




item	ordercode	description	quantity
-	SP1423177	HOLDER F. PUSH HANDLE RECARO CPL.	1
-	NA	SEAT SETTING STRAP	1
-	NA	TUBE END PLUG SL 22	2
-	NA	TUBE END PLUG E 15 X 15	2
1	NA	HEX HEAD SCREW M8 X 20	2
2	NA	DIN 936 M8 17H BLACK GALVANISED HEX NUT THIN	2
2	NA	NYLOC NUT M6	2
3	NA	HEX HEAD SCREW M6 X 40	4
4	NA	HOLDER FOR PUSH HANDLE RECARO; RAL 9005, DEEP BLACK	1
5	NA	WASHER 6,4 X 12,5 X 1,6	4
6	NA	HANDLE; RAL 9005 DEEP BLACK	2
7	L10009	HANDLE	2

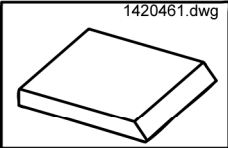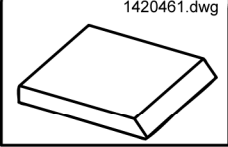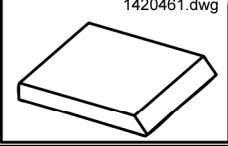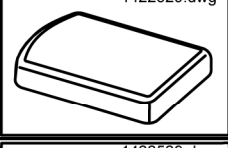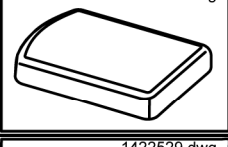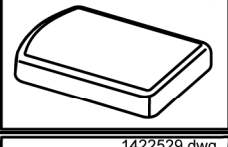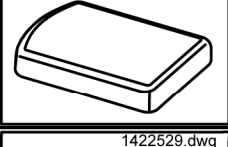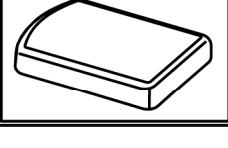
Seat unit - Cushion




Picture	Seat system	Seat width cm	Seat depth cm	color	ordercode
	Standard	n.a.	41	black	1420460
	Standard	n.a.	46	black	1420461
	Standard	n.a.	51	black	1420462
	Flex	39	n.a.	Dartex grey	5252582-TR23
	Flex	39	n.a.	Vinyl grey	5314542
	Flex	39	n.a.	blue	1422529
	Flex	39	n.a.	black	1436561
	Flex	43	n.a.	Dartex grey	5252583-TR23
	Flex	43	n.a.	Vinyl grey	5314525
	Flex	41	n.a.	blue	1422530
	Flex	41	n.a.	black	1436562

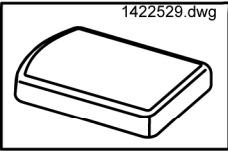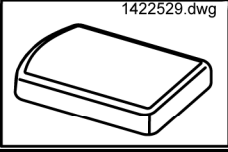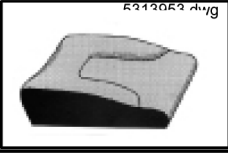Picture	Seat system	Seat width	Seat depth	color	ordercode
		cm	cm		
	Flex	48	n.a.	Dartex grey	5252584-TR23
	Flex	48	n.a.	Vinyl grey	5314526
	Flex	48	n.a.	blue	1422532
	Flex	48	n.a.	black	1436563
	Flex Vic Air	43	n.a.	VicAir	1422531
	Kontur	43	n.a.	black	1422072
	Kontur	48	n.a.	black	1422074
	Sitzschutz Comfort Flex	39	n.a.	blue	5253078-TR25
	Sitzschutz Comfort Flex	43	n.a.	blue	5253079-TR25
	Sitzschutz Comfort Flex	48	n.a.	blue	5253080-TR25
	positiv	45	n.a.	black	1419632
	positiv	45	n.a.	black	5313953
	positiv	45	n.a.	black	5314038
	negativ	45	n.a.	black	1419633











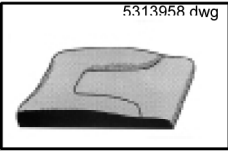
Picture	Seat system	Seat width	Seat depth	color	ordercode
		cm	cm		
 5313958 r/wg	negativ	45	n.a.	black	5313958
 5313962 r/wg	paraplegic	45	n.a.	black	1419634
 5313962 r/wg	paraplegic	45	n.a.	black	5313962
 5313962 r/wg	paraplegic	45	n.a.	black	5314044







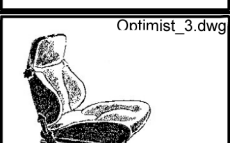
Seat unit - Arne Jensen-Seats

Picture	Typ	Seat height cm	Seat width cm	el. strap	ordercode
	1	46	38	-	1419630
	1	46	45	-	5313888
	1	60	45	-	5313889
	1	60	50	-	1419625
	2	46	38	-	1419631
	2	46	38	X	5313867
	2	46	45	-	5313882
	2	60	45	-	5313883
	2	60	45	X	5313895
	2	60	50	-	1419628
	3	60	45	-	5313878







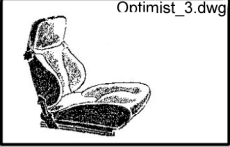
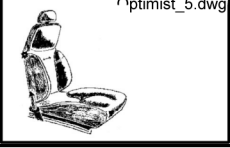
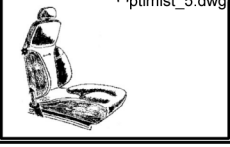
Picture	Typ	Seat height cm	Seat width cm	el. strap	ordercode
 Optimist_3.dwg	3	70	50	-	5313877
 Optimist_5.dwg	5	60	45	-	5313872
 Optimist_5.dwg	5	70	50	-	5313871

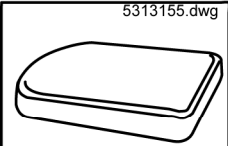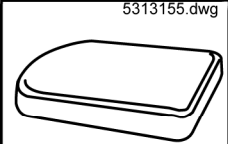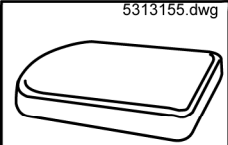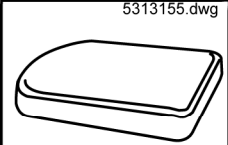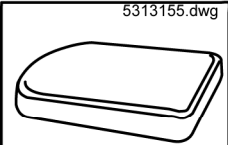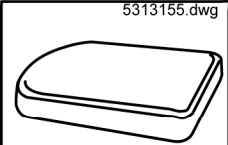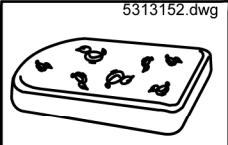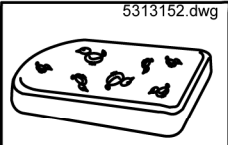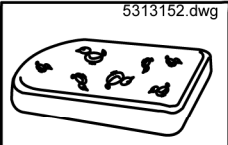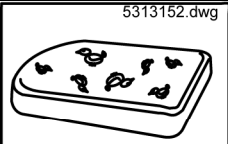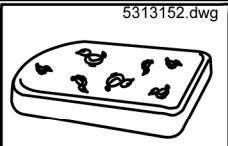
Seat unit (children) - Seat and Back Cushions

Picture	color	Seat system	Seat width cm	ordercode
	black	Standard	30	5313155
	black	Standard	35	5313181
	black	Standard	38	5313207
	black	Ergo	30	5313159
	black	Ergo	35	5313185
	black	Ergo	38	5313211
	blue	Standard	30	5313152
	blue	Standard	35	5313178
	blue	Standard	38	5313204
	blue	Ergo	30	5313156
	blue	Ergo	35	5313182

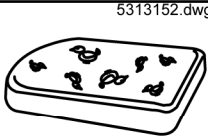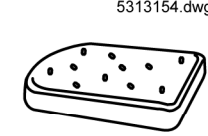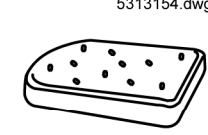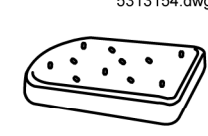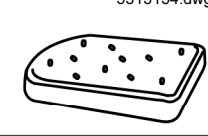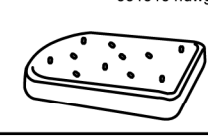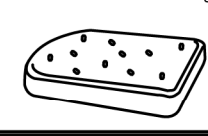
Picture	color	Seat system	Seat width cm	ordercode
5313152.dwg 	 blue	Ergo	38	5313208
5313154.dwg 	 blue	Standard	30	5313154
5313154.dwg 	 blue	Standard	35	5313180
5313154.dwg 	 blue	Standard	38	5313206
5313154.dwg 	 blue	Ergo	30	5313158
5313154.dwg 	 blue	Ergo	35	5313184
5313154.dwg 	 blue	Ergo	38	5313210

Picture	color	color	Seat system	Seat width cm	ordercode
5313139.dwg 	blue Birdie	 blue	Standard	30	5313142
5313139.dwg 	blue Birdie	 blue	Standard	35	5313168
5313139.dwg 	blue Birdie	 blue	Standard	38	5313194
5313141.dwg 	blue Dot	 blue	Standard	30	5313146

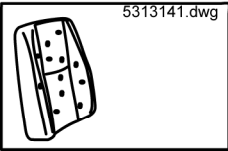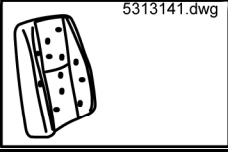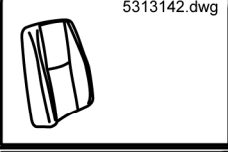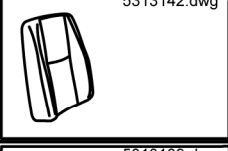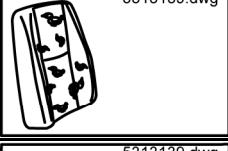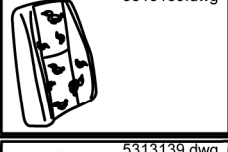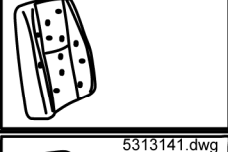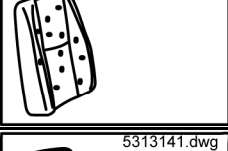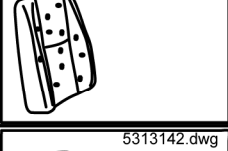Picture	color	color	Seat system	Seat width cm	ordercode
	blue Dot		Standard	35	5313172
	blue Dot		Standard	38	5313198
	gray Vinyl		Standard	30	5313139
	gray Vinyl		Standard	35	5313165
	gray Vinyl		Standard	38	5313191
	blue Birdie		Ergo	30	5313143
	blue Birdie		Ergo	35	5313169
	blue Birdie		Ergo	38	5313195
	blue Dot		Ergo	30	5313141
	blue Dot		Ergo	35	5313167
	blue Dot		Ergo	38	5313193
	gray Vinyl		Ergo	30	5313145

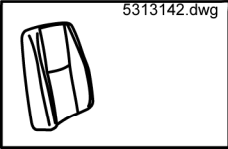Picture	color	color	Seat system	Seat width cm	ordercode
 5313142.dwg	gray Vinyl	 grey	Ergo	35	5313171
 5313142.dwg	gray Vinyl	 grey	Ergo	38	5313197

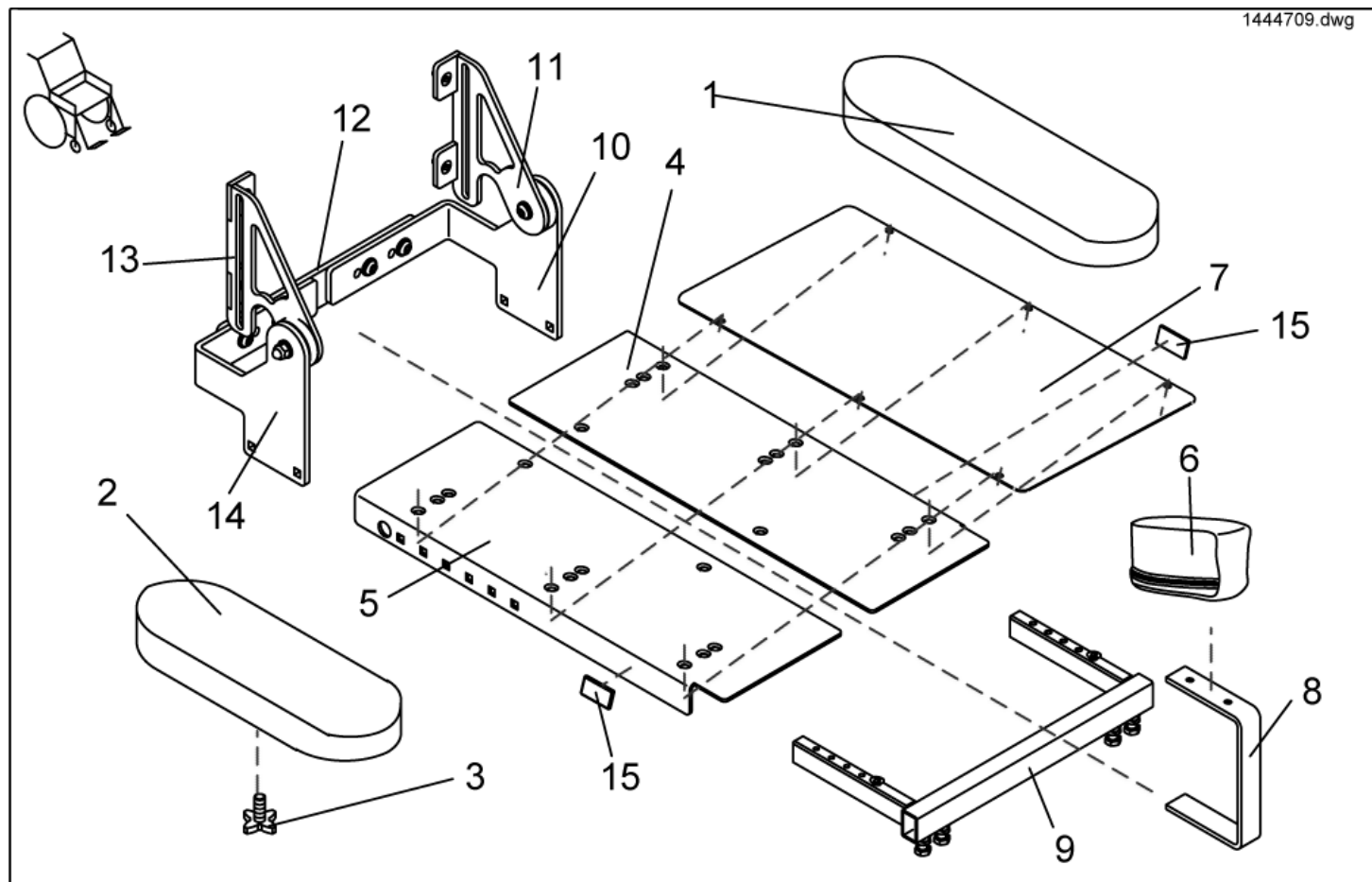Seat unit (children) - Seat unit and side parts




item	ordercode	description	quantity
-	5313558	ARMREST FOR JUNIOR SEAT LH CPL.	1
-	5313557	ARMREST FOR JUNIOR SEAT RH	1
-	SP1444709	SEAT FRAME CHILDREN SEAT, COMPLETE WITH BACKREST SUPPORT AND LEGREST SUPPORT	1
-	SP1445031	EXPANSION WEDGE UPHOLSTERY CHILDREN "BLUE BIRDY", CPL.	1
-	SP1445032	EXPANSION WEDGE UPHOLSTERY CHILDREN "BLUE DOT", CPL.	1
-	SP1445033	EXPANSION WEDGE UPHOLSTERY CHILDREN "VINYL GREY", CPL.	1
-	5370088	SAFETY PELVIC BELT	1

item	ordercode	description	quantity
1	5313242	ARMREST UPHOLSTERY LH	1
2	5313241	ARMREST UPHOLSTERY RH	1
3	5824117	HAND WHEEL M8X15	2
4	1442830-9005	SEAT PLATE CHILDREN, LEFT	1
5	1442831-9005	SEAT PLATE CHILDREN, RIGHT	1
6	5313254	EXPANSION WEDGE UPHOLSTERY "BLUE BIRDY"	1
6	5313256	EXPANSION WEDGE UPHOLSTERY "BLUE DOT"	1
6	5313257	EXPANSION WEDGE UPHOLSTERY "VINYL BLACK"	1
7	1449086-9005	SEAT SHEET METAL CHILD SEAT RAL 9005	1
8	1436402-9005	ABDUKTIONS WEDGE HOLDER	1
9	1442941-9005	LEGRESTS ADMISSION	1
10	NA	BACK HOLDER LH	1
11	5313224	SCHANIER ATTACHMENT	1
12	NA	ACTUATOR ADMISSION	1
13	5313223	SCHANIER ATTACHMENT	1
14	NA	BACK HOLDER RH	1
15	P31298	REFLECTOR, YELLOW	2

Seat unit (children) - Back Unit

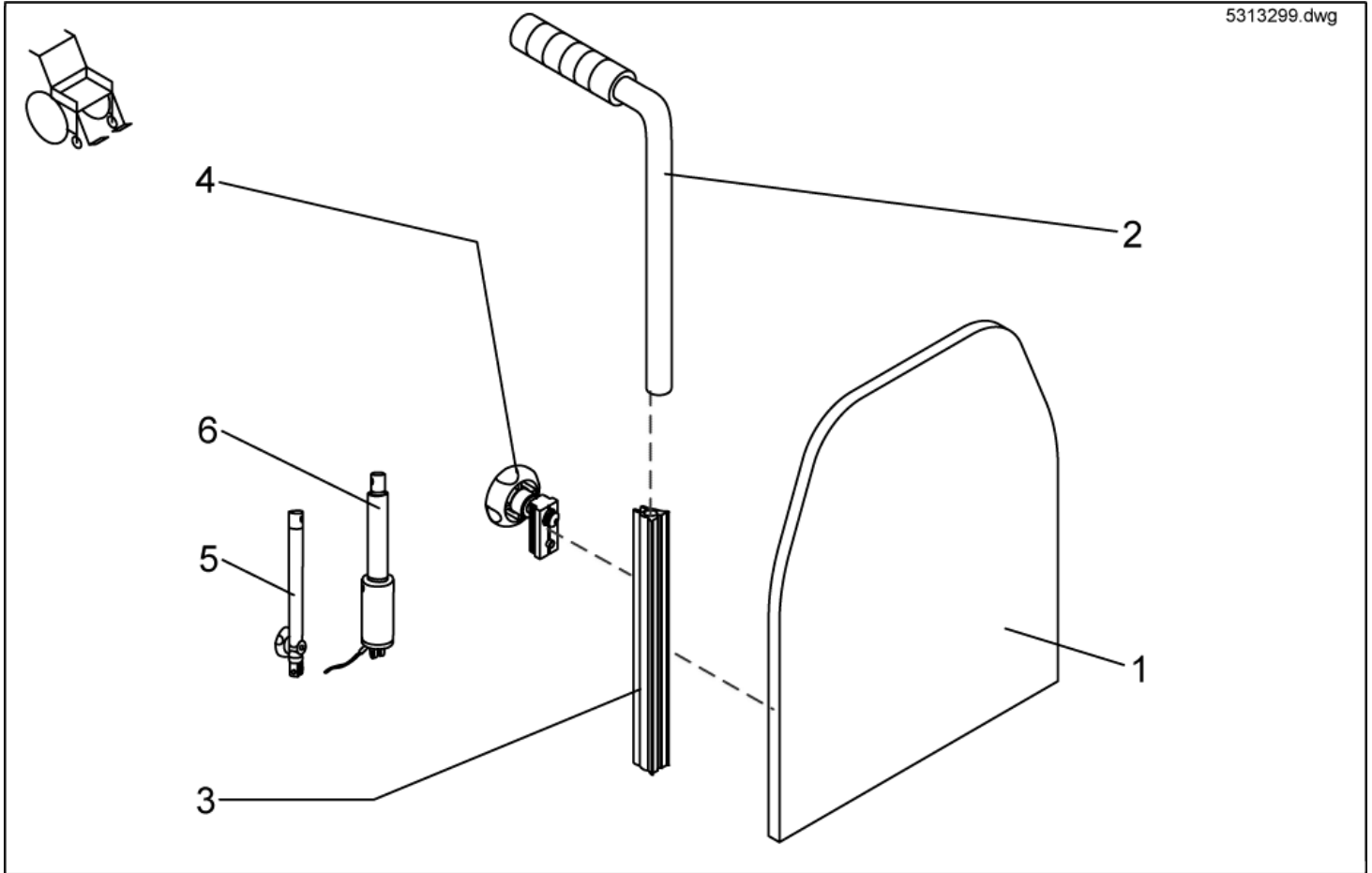
5313299.dwg



item	ordercode	description	quantity
1	5313136	BACK PLATE 30 cm	1
1	5313162	BACKREST PLATE SB 35	1
1	5313188	BACKREST PLATE SB 38	1
2	5253056	PUSH HANDLE	1
3	5313299	BACKREST BAR CPL.	1
4	5253088	FIXING BLOCK	2
5	SP1445030	MANUAL BACKREST ADJUSTMENT WITH MOUNTING KIT	1
5	5313220	BACKREST ADJUSTMENT MANUAL	1
6	SP1445029	ELECTRICAL BACKREST ADJUSTMENT WITH MOUNTING KIT	1
6	5316008	MOTOR (ACS)	1

Seat unit (children) - Trunk Support/ Head Rest

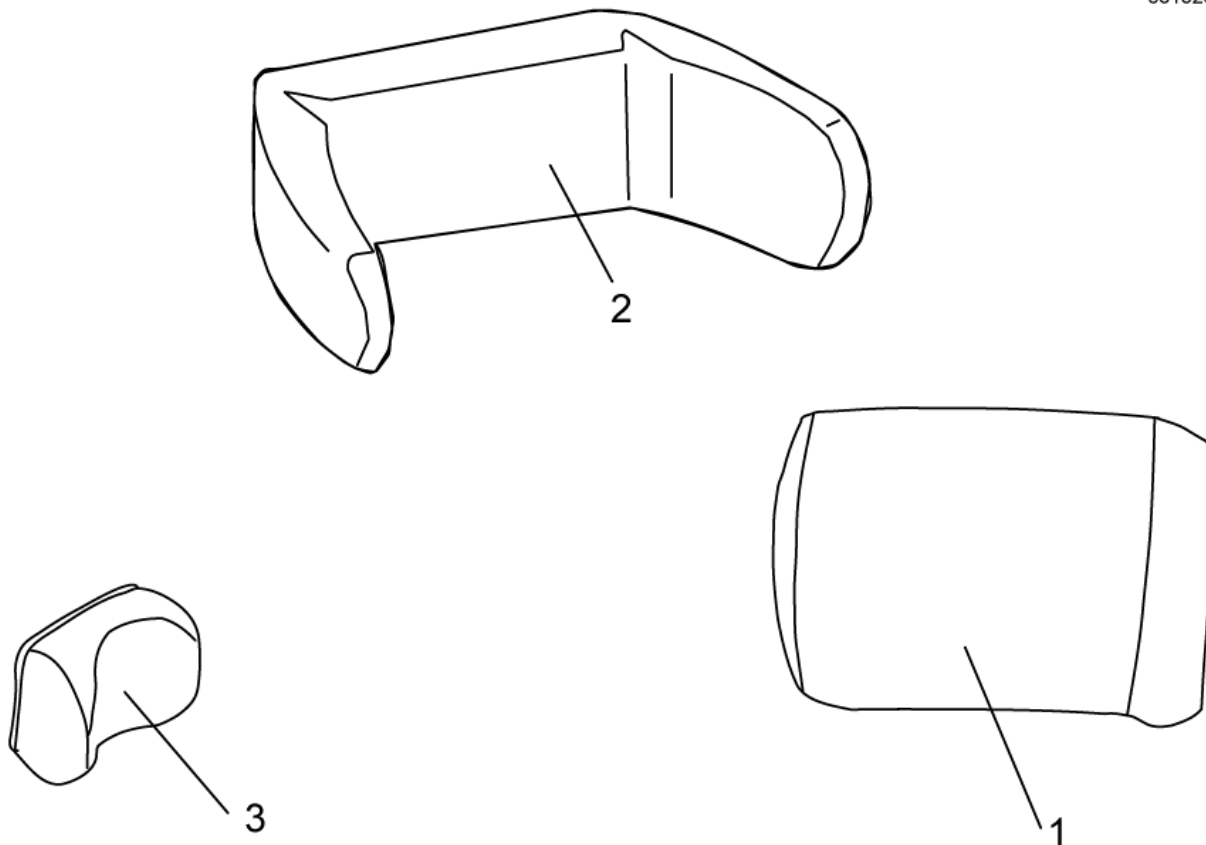
5313269.dwg



item	ordercode	description	quantity
1	5313269	TRUNK SUPPORT "BLUE BIRDY" RH CPL.	1
1	5313271	TRUNK SUPPORT "BLUE DOT" RH CPL.	1
1	5313272	TRUNK SUPPORT "VINYL BLACK" RH CPL.	1
1	5313273	TRUNK SUPPORT "BLUE BIRDY" LH CPL.	1
1	5313275	TRUNK SUPPORT "BLUE DOT" LH CPL.	1
1	5313276	TRUNK SUPPORT "VINYL BLACK" LH CPL.	1
2	5313305	HEADREST "BLUE BIRDY" CPL.	1
2	5313307	HEADREST "BLUE DOT" CPL.	1
2	5313308	HEADREST "VINYL BLACK" CPL.	1
2	5313259	HEADREST UPHOLSTERY "BLUE BIRDY"	1
2	5313261	HEADREST UPHOLSTERY "BLUE DOT"	1
2	5313262	HEADREST UPHOLSTERY "VINYL BLACK"	1
3	5313309	THORACIC SUPPORT "BLUE BIRDY" CPL.	1
3	5313311	THORACIC SUPPORT "BLUE DOT" CPL.	1
3	5313312	THORACIC SUPPORT "VINYL BLACK" CPL.	1
3	5313263	THORACIC UPHOLSTERY "BLUE BIRDY"	1
3	5313265	THORACIC UPHOLSTERY "BLUE DOT"	1
3	5313266	THORACIC UPHOLSTERY "VINYL BLACK"	1
-	5255497	HEADREST HOLDER	1
-	5221122	HANDWHEEL	1

Seat unit (children) - Desk

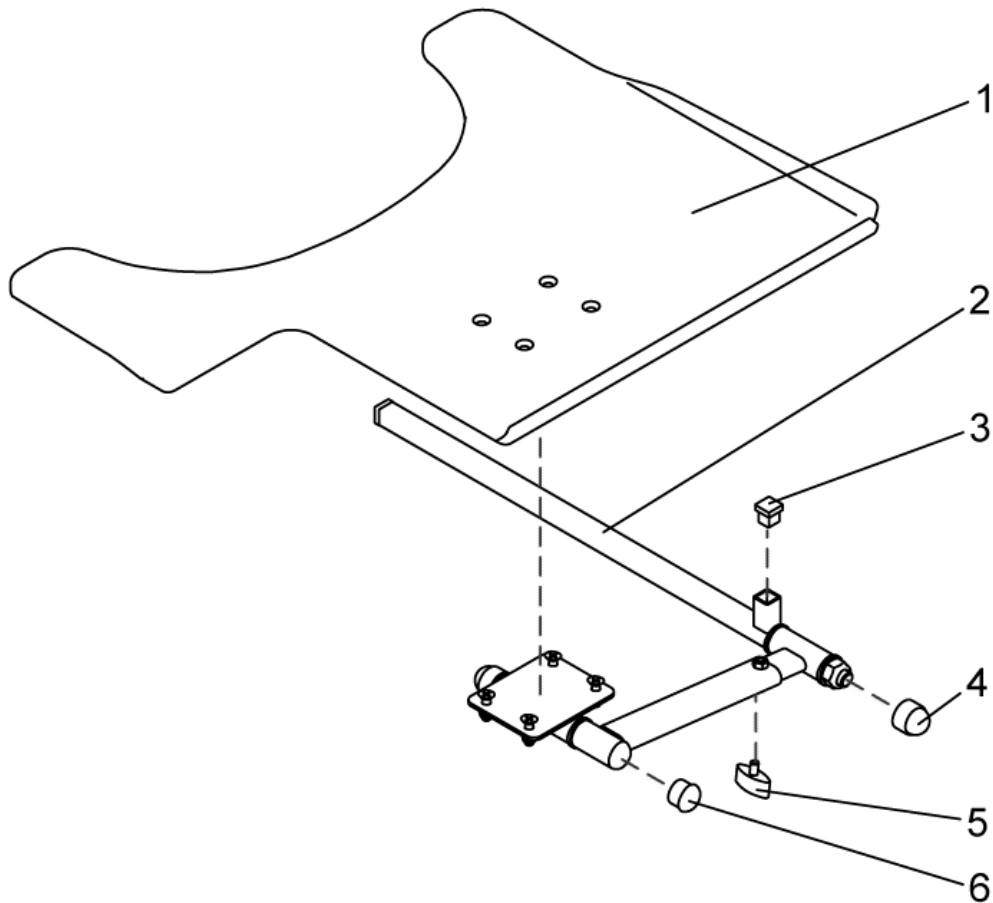
N377001.dwg




item	ordercode	description	quantity
-	SPN377001	TABLE RH FOLDAWAY WITH HOLDER WITH CUT-OUT FOR REMOTE LH	1
-	SPN377002	TABLE LH FOLDAWAY WITH HOLDER WITH CUT-OUT FOR REMOTE RH	1
1	4100007	TABLETOP 540 x 420 x 8 MM WITHOUT CUT-OUT	1
1	1415084	TABLETOP CUT LH	1
1	1415083	TABLETOP CUT RH	1
2	1446698	TRAY SUPPORT FIXING KIT CPL.	1
3	L01055	TUBE END PLUG E 15 X 15	2
4	L01037	NUT CAP M12	2
5	L31001	HAND KNOB M6 X 10	1
6	L01008	TUBE END PLUG SL 22	1

Seat unit (children) - Legrests

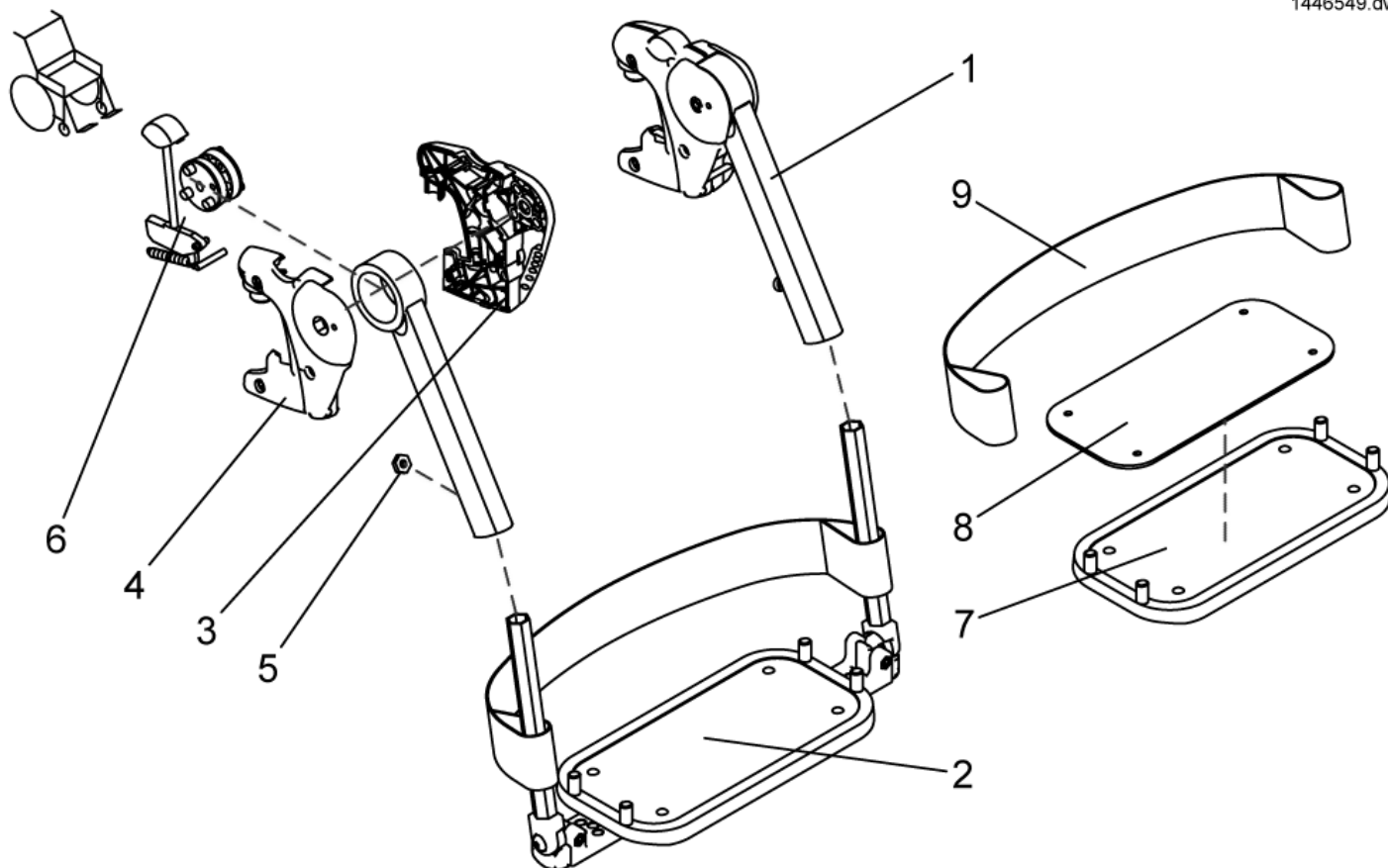1446549.dwg




item	ordercode	description	quantity
1	N373015	LEG SUPPORT UPPER PART LH	1
1	N373016	LEG SUPPORT UPPER PART RH	1
2	N374001	FOOTBOARD 240 MM CPL.	1
2	N374002	FOOTBOARD 280 MM CPL.	1
3	Z173181	HANGER LH	1
4	Z173182	HANGER RH	1
5	Z173189.99	NUT M6 SW-14	2
6	SP1423193	RELEASE UNIT ASSEMBLY KIT	2
7	1446696	FOOTPLATE 240 MM WITH RUBBER MAT	1
7	1446697	FOOTPLATE 280 MM WITH RUBBER MAT	1
8	0570-04049	RUBBER MAT 240 MM	1
8	0570-04050	RUBBER MAT 280 MM	1
9	N373013	CALF STRAP	1

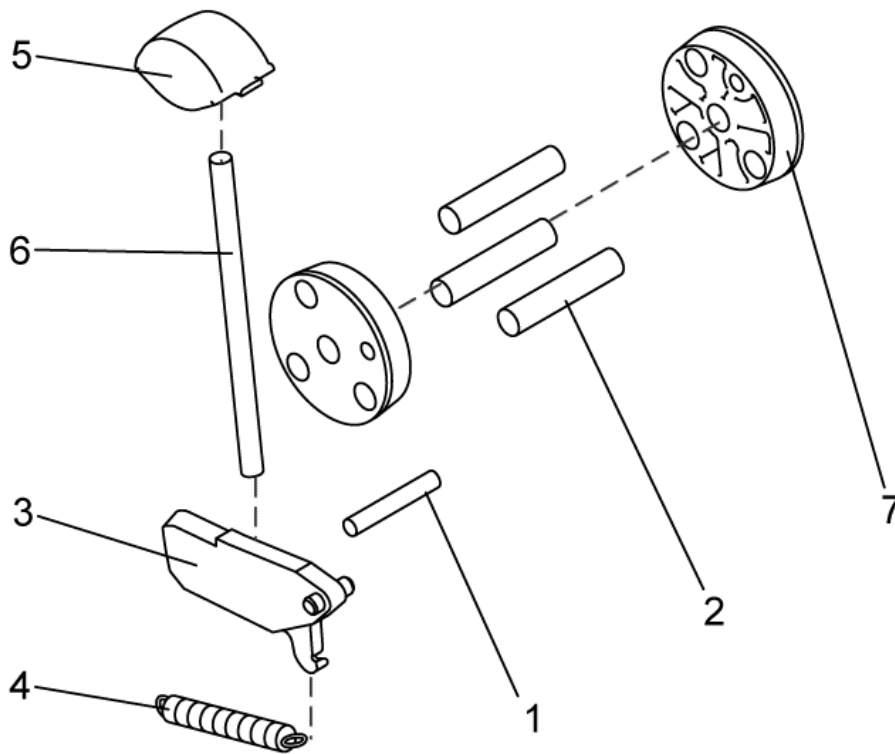

item	ordercode	description	quantity
1-7	SP1423193	RELEASE UNIT ASSEMBLY KIT	2
1	NA	DOWEL RD. 5 X 32; DIN-7	1
2	NA	DOWEL RD. 8 X 40; DIN-7	3
3	NA	LOCK LEVER	1
4	NA	SPRING	1
5	NA	RELEASE KNOB	1
6	NA	PUSH ROD	1
7	NA	CONED WASHER	2













Side Panels - Standard Side Panels


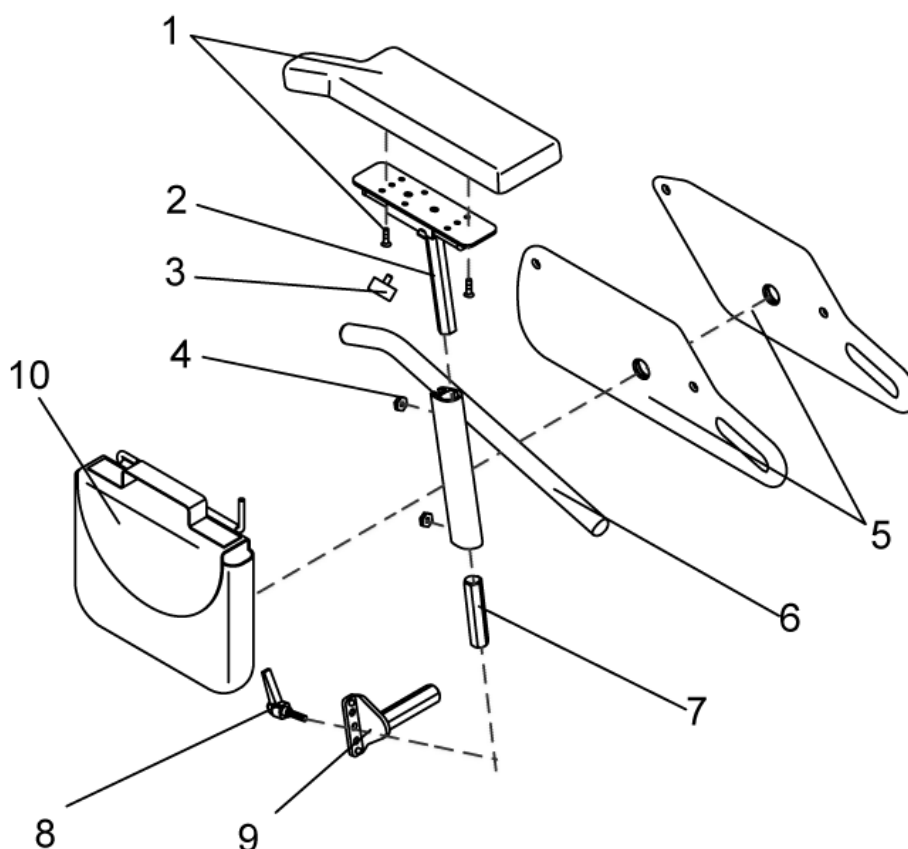
Seitenteile.dwg



item	ordercode	description	quantity
1-9	NA	ARMREST ASSEMBLIES COMPLETE »»» SEE "STANDARD ARMREST ASSEMBLIES CPL."	2
1	SP1423224	ARMREST PAD LH CPL.	1
1	SP1423221	ARMREST PAD RH CPL.	1
2	1421717-9005	ARM PAD RECEPTACLE LH	1
2	1421716-9005	ARM PAD RECEPTACLE RH	1
3	L31001	HAND KNOB M6 X 10	4
4	Z173189.99	NUT M6 SW-14	4
5	NA	»»» see "SIDE PART PLATES COMPLETE WITH ADJUSTMENT"	
6	1422095-9005	SIDE PART ADJUSTMENT LH CPL. 250-340 MM	1
6	1422096-9005	SIDE PART ADJUSTMENT RH CPL. 250-340 MM	1
6	1422099-9005	SIDE PART ADJUSTMENT LH CPL. 290-380 MM	1
6	1422098-9005	SIDE PART ADJUSTMENT RH CPL. 290-380 MM	1
7	1422092-0002	CONNECTION PIECE	2
8	1415027	CLAMP LEVER M6 X 25	2
9	NA	»»» see "SIDE PART RECEPTACLES COMPLETE"	
9	NA	»»» see "SIDE PART RECEPTACLES SINGLE"	
10	SP1430417	BAG FOR SIDE PANEL LH CPL.	1
10	SP1430416	BAG FOR SIDE PANEL RH CPL.	1

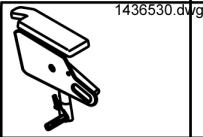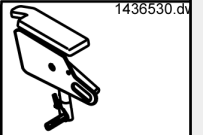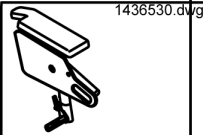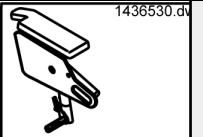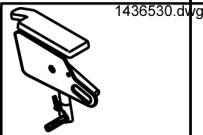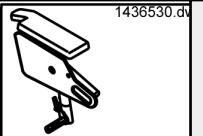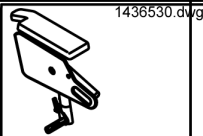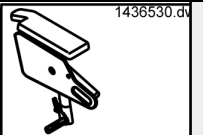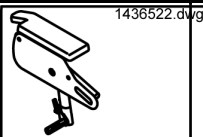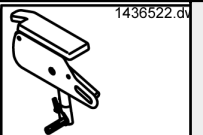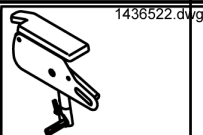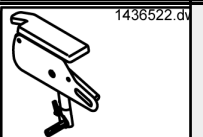Side Panels - Side Panels for low seatheight (38cm Tornado/ 40cm Typhoon)

Picture	Seat system	Seat depth cm	Height cm	Seat width cm	left/ right	ordercode
	Flex	41	34	39	LH	SP1436534
	Flex	41	34	39	RH	SP1436533
	Standard/Kontur/Flex	41	34	39...	LH	SP1436530
	Standard/Kontur/Flex	41	34	39...	RH	SP1436529
	Flex	41	38	39	LH	SP1436536
	Flex	41	38	39	RH	SP1436535
	Standard/Kontur/Flex	41	38	39...	LH	SP1436532
	Standard/Kontur/Flex	41	38	39...	RH	SP1436531
	Flex	41...	34	39	LH	SP1436526
	Flex	41...	34	39	RH	SP1436525
	Flex	41...	38	39	LH	SP1436528
	Flex	41...	38	39	RH	SP1436527


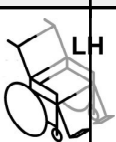
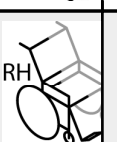
Picture	Seat system	Seat depth	Height	Seat width	left/ right	ordercode
		cm	cm	cm		
	Standard/Kontur/Flex	41...	38	39...		SP1436524
	Standard/Kontur/Fle	41...	38	39...		SP1436523
	RECARO	n.a.	38	n.a.		SP1436538
	RECARO	n.a.	38	n.a.		SP1436537

✓ **Information:** The height of the side parts is setable. Preset is 34cm or 38cm.

- Parts with **34cm** are setable between **30cm and 40cm**.
- Parts with **38cm** are setable between **34cm and 44cm**.


Side Panels - Side Panels for Standard seatheight

Picture	Seat system	Seat depth cm	Height cm	Seat width cm	left/ right	ordercode
	Flex	41	34	39		SP1428101
	Flex	41	34	39		SP1428100
	Standard/Kontur/Flex	41	34	39...		SP1428097
	Standard/Kontur/Flex	41	34	39...		SP1428096
	Flex	41	38	39		SP1428103
	Flex	41	38	39		SP1428102
	Standard/Kontur/Flex	41	38	39...		SP1428099
	Standard/Kontur/Flex	41	38	39...		SP1428098
	Flex	41...	34	39		SP1422543
	Flex	41...	34	39		SP1422542
	Standard/Kontur/Flex	41...	34	39...		SP1421694
	Standard/Kontur/Flex	41...	34	39...		SP1421692

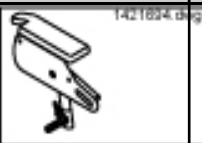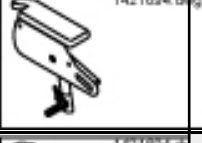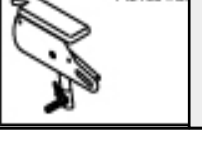Picture	Seat system	Seat depth	Height	Seat width	left/ right	ordercode
		cm	cm	cm		
	Flex	41...	38	39		SP1422549
	Flex	41...	38	39		SP1422548
	Standard/Kontur/Flex	41...	38	39...		SP1421695
	Standard/Kontur/Fle	41...	38	39...		SP1421693
	RECARO	n.a.	38	n.a.		SP1425846
	RECARO	n.a.	38	n.a.		SP1425845






- ✓ **Information:** The height of the side parts is setable. Preset is 34cm or 38cm.
- Parts with **34cm** are setable between **30cm and 40cm**.
 - Parts with **38cm** are setable between **34cm and 44cm**.

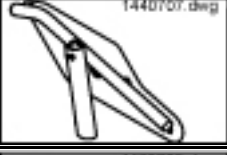Side Panels - Side part plates complete with side part adjustment









Picture	Seat depth cm	Height cm	left/ right	ordercode
	41	34		1440707
	41	34		1440708
	41	38		1440705
	41	38		1440706
	41...	34		SP1423702
	41...	34		SP1423222
	41...	38		SP1423703
	41...	38		SP1423701

- ✓ **Information:** The height of the side parts is settable. Preset is 34cm or 38cm.
- Parts with **34cm** are settable between **30cm and 40cm**.
 - Parts with **38cm** are settable between **34cm and 44cm**.

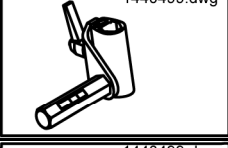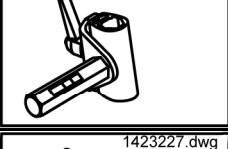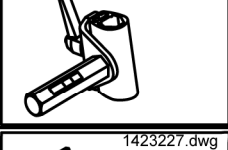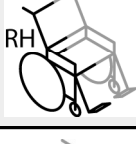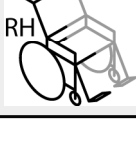Side Panels - Side part receptacles complete

Picture	Seat height cm	Seat width cm	left/ right	ordercode
 1440499.dwg	38/40	39	 LH	1440499
 1440499.dwg	38/40	39	 RH	1440498
 1440499.dwg	38/40	39...	 LH	1440501
 1440499.dwg	38/40	39...	 RH	1440500
 1423227.dwg	38/40...	39	 LH	SP1423227
 1423227.dwg	38/40...	39	 RH	SP1423226
 1423227.dwg	38/40...	39...	 LH	SP1423225
 1423227.dwg	38/40...	39...	 RH	SP1423223

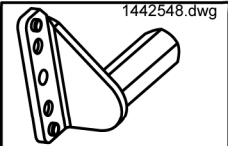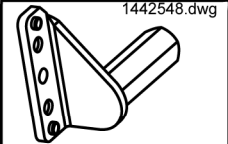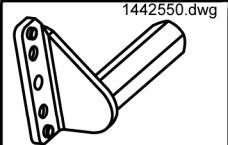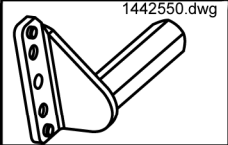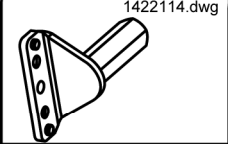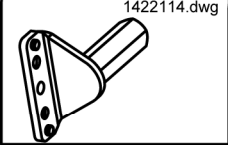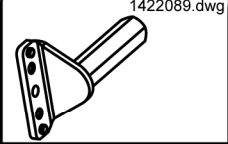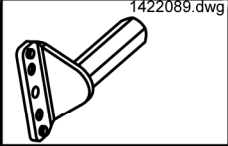Side Panels - Side part receptacles single


Picture	Seat height cm	Seat width cm	left/ right	ordercode
	38/40	39	 LH	1442547-9005
	38/40	39	 RH	1442548-9005
	38/40	39...	 LH	1442549-9005
	38/40	39...	 RH	1442550-9005
	38/40...	39	 LH	1422115-9005
	38/40...	39	 RH	1422114-9005
	38/40...	39...	 LH	1422097-9005
	38/40...	39...	 RH	1422089-9005

Side Panels - Side Panels for Arne Jensen-Seats

Picture	length mm	left/ right	ordercode
 1419658.dwg	300		1419658
 1419658.dwg	300		1419657
 1419658.dwg	350		1419660
 1419658.dwg	350		1419659
 1419658.dwg	400		1419662
 1419658.dwg	400		1419661
 1419648.dwg	300		1419648
 1419648.dwg	300		1419647
 1419648.dwg	350		5314010S
 1419648.dwg	350		5314011
 1419648.dwg	400		5314010


Picture	length mm	left/ right	ordercode
	400		5314011L

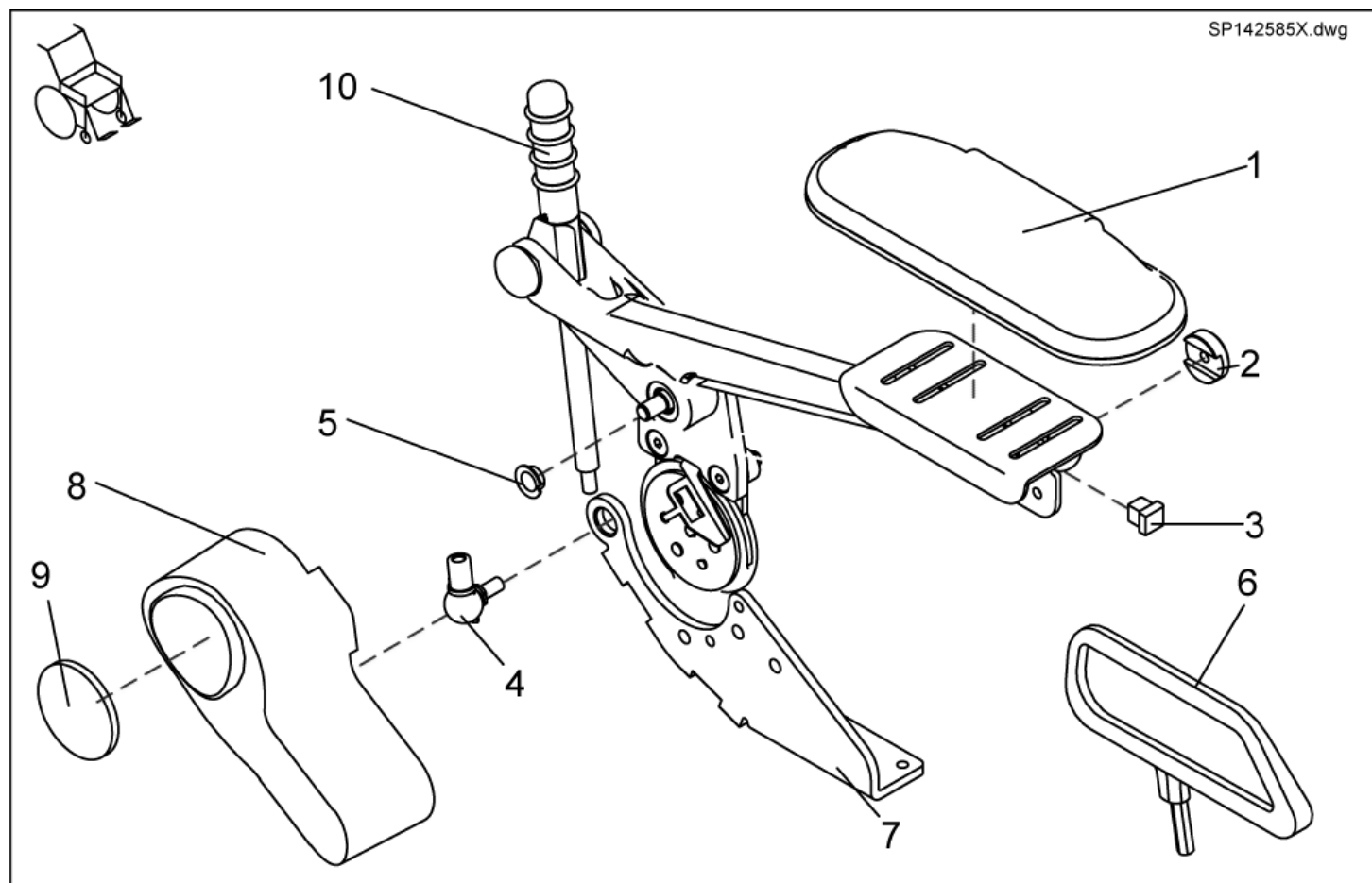Side Panels - Sliding Armrests



item	ordercode	description	quantity
-	SP1425853	KIT SLIDING ARMRESTS PAIR MANUAL	1
-	SP1425859	KIT SLIDING ARMRESTS PAIR, F. EL. BACKREST ADJUSTMENT	1
1,2,3,4,5	SP5314641	SLIDING ARMREST LH CPL.	1
1,2,3,4,5	SP5311680	SLIDING ARMREST RH CPL.	1
1	5311680	ARMREST UPHOLSTERY 260 MM	1
1	5314641	ARMREST UPHOLSTERY 320 MM	1
2	1421657-0003	RETAINING WASHER BLACK GALVANISED	4
3	L01055	TUBE END PLUG E 15 X 15	1
4	N04109	SWIVEL JOINT AS 13-M8 RH	2
5	N02194	BUSH BB 1209 DU	8
6	SP1436540	CLOTHING PROTECTOR PAIR SLIDING ARMREST SB=39CM	1
6	SP1436539	CLOTHING PROTECTOR PAIR SLIDING ARMREST SB>39CM	1
7	1429564-9005	FITTING ATTACHMENT LH F. SLIDING ARMREST	1
7	1429563-9005	FITTING ATTACHMENT RH F. MANUAL ADJUSTMENT	1
8	1428057	MOTOR CAP FOR ELECTRIC BACKREST	1
9	1420482	REFLECTOR	1
10	SP1451392	ASSEMBLY KIT: GUIDE ROD	1

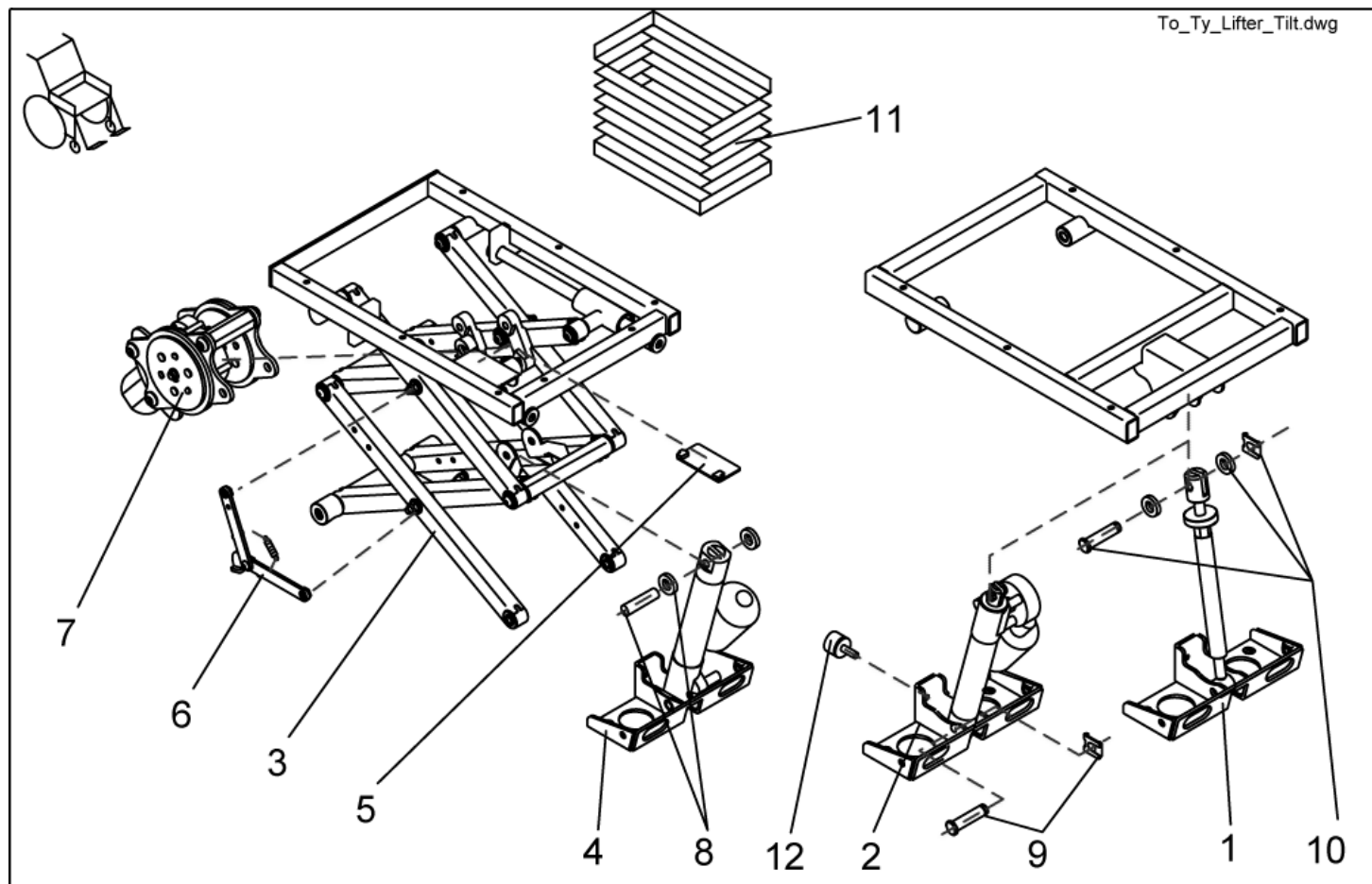
Lifter



item	ordercode	description	quantity
1	SP1440313	SPINDLE MANUEL TILT	1
1	1420004	SPINDLE SEAT ANGLE ADJUSTMENT MANUAL CPL.	1
2	SP1440463	TILT ACTUATOR ASSY	1
2	1434913	ACTUATOR 120/270	1
3	SP1440317	LIFTER AND TILT WITH STABILITY LOCK COMPLETE	1
3	SP1440468	LIFTER AND NEGATIVE TILT WITH STABILITY LOCK COMPLETE	1
4	SP1440464	LIFTER ACTUATOR ASSY (120/270)	1
4	1434913	ACTUATOR 120/270	1
4	SP1440311	LIFTER ACTUATOR ASSY (120/300)	1
4	1436555	ACTUATOR 120/300	1
5	1436266	LIMIT STOP SWITCH CIRCUIT BOARD complete with endswitch for tilt and various cables	1
6	1442585	SETTING UP ASSISTANCE CPL.	1
7	1442740	ADJUSTING UNIT FOR TILT, ASSEMBLED	1
8	1444672	BOLT with two SPACER	1
9	1444673	SECURING BOLT with SL-LOCK	1
10	1444674	SECURING BOLT with two SPACER and SL-LOCK	1
11	SP1445036	GAITER	1
12	1435031	BUFFER PLATE (RUBBER)	1

Control Unit/ Joystickbox

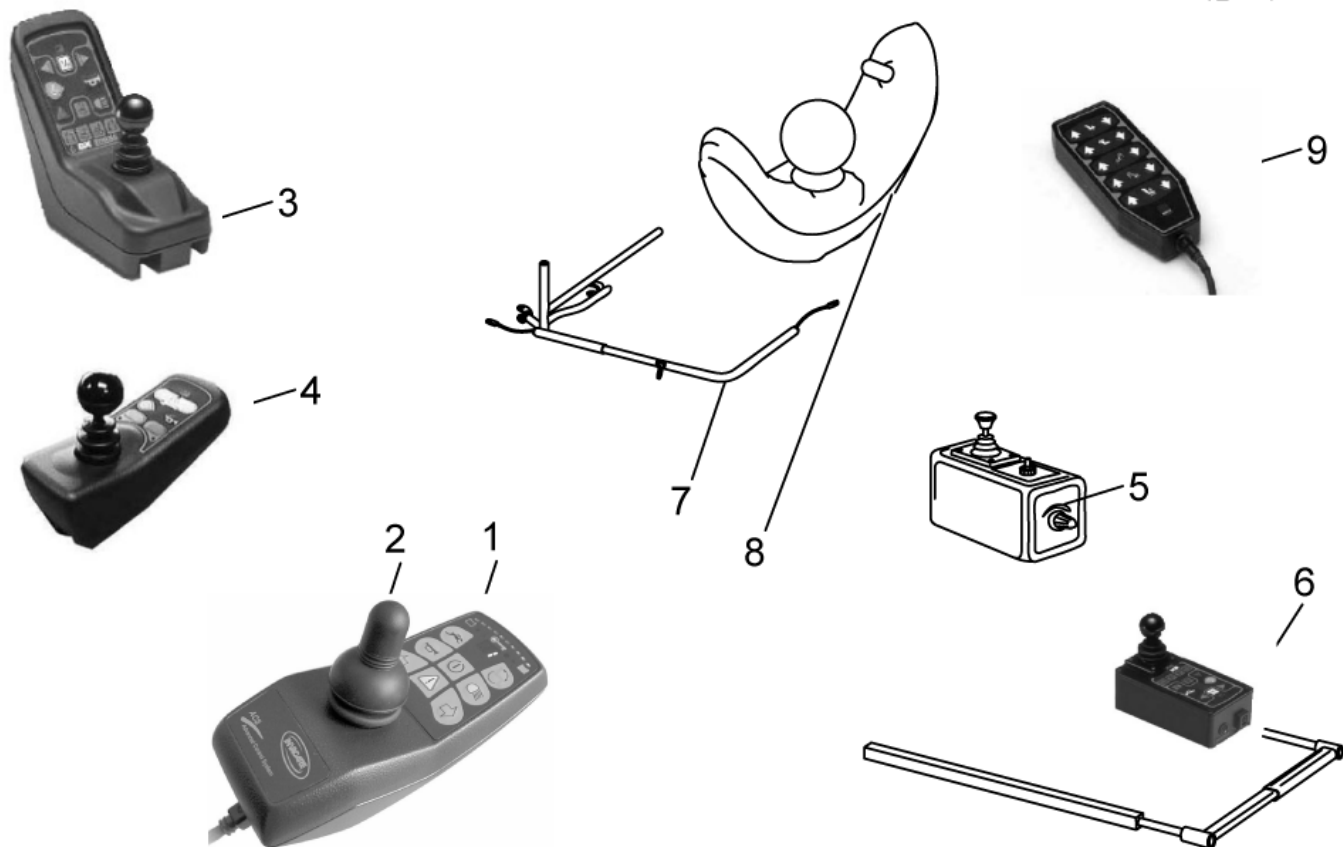
Ty_Fahrpulte.dwg




item	ordercode	description	quantity
1	1436541	REMOTE REM 24 SD	1
2	Z183229	JOYSTICK KNOB CPL.	1
	SP1422828	GUARD BOW F. REM 24 REMOTE CPL.	1
	GSM63003	ACS BUS CABLE L=300 MM	1
	GSM63016	ACS-BUS CABLE L=1600 MM	1
	1440424	ACS-BUS CABLE LONG VERSION	1
3	1419205	JOYSTICK BOX REM 34 (DOLPHIN)	1
4	P10156	REMOTE REM G80 I	1
	SP1422827	GUARD BOW F. G80I REMOTE CPL.	1
5	6001425	ACS CTRL. FOR ATTENDANT SWITCHABLE	1
	SP1436544	ATTENDANT JOYSTICK BOX REM 24SD	1
6	SP1422799	SWING-OUT MIDDLE CONTROL LH CPL.	1
6	SP1422800	SWING-OUT MIDDLE CONTROL RH CPL.	1
	1422947	ASSEMBLY KIT F. CHIN CONTROL HOLDER CPL.	1
	SP1422951	ASSEMBLY KIT F. CHIN CONTROL HOLDER RECARO CPL.	1
7	SP1422943	ASSEMBLY KIT FOR ELECTR. HORIZONTAL SWING-OUT CHIN CONTROL	1
7	SP1422942	ASSEMBLY KIT RH F. MANUAL HORIZONTAL SWING-OUT CHIN CONTROL	1
7	SP1422945	ASSEMBLY KIT FOR ELECTR. HORIZONTAL SWING-OUT CHIN CONTROL, LH CPL.	1
7	SP1422944	ASSEMBLY KIT RH F. EL. HORIZONTAL SWING-OUT CHIN CONTROL	1
7	SP1422946	ELECTR. VERTICAL SWING-OUT CHIN CONTROL ASSEMBLY KIT	1
8	SP1422801	ACS CHIN CONTROL LH CPL.	1
8	SP1422802	ACS CHIN CONTROL RH CPL.	1
9	F282007	ARC5 CONTROL CPL.	1
	6001450	ACS-HAND PROGRAMMER, "F; GER; ESP"	1
----	P10062	WIZARD SOFTWARE	1
----	SP1419695	MAGNETIC KEY W. POUCH CPL.	1
----	1446541	USB Wizard Dongle Kit	1

Attendant Control

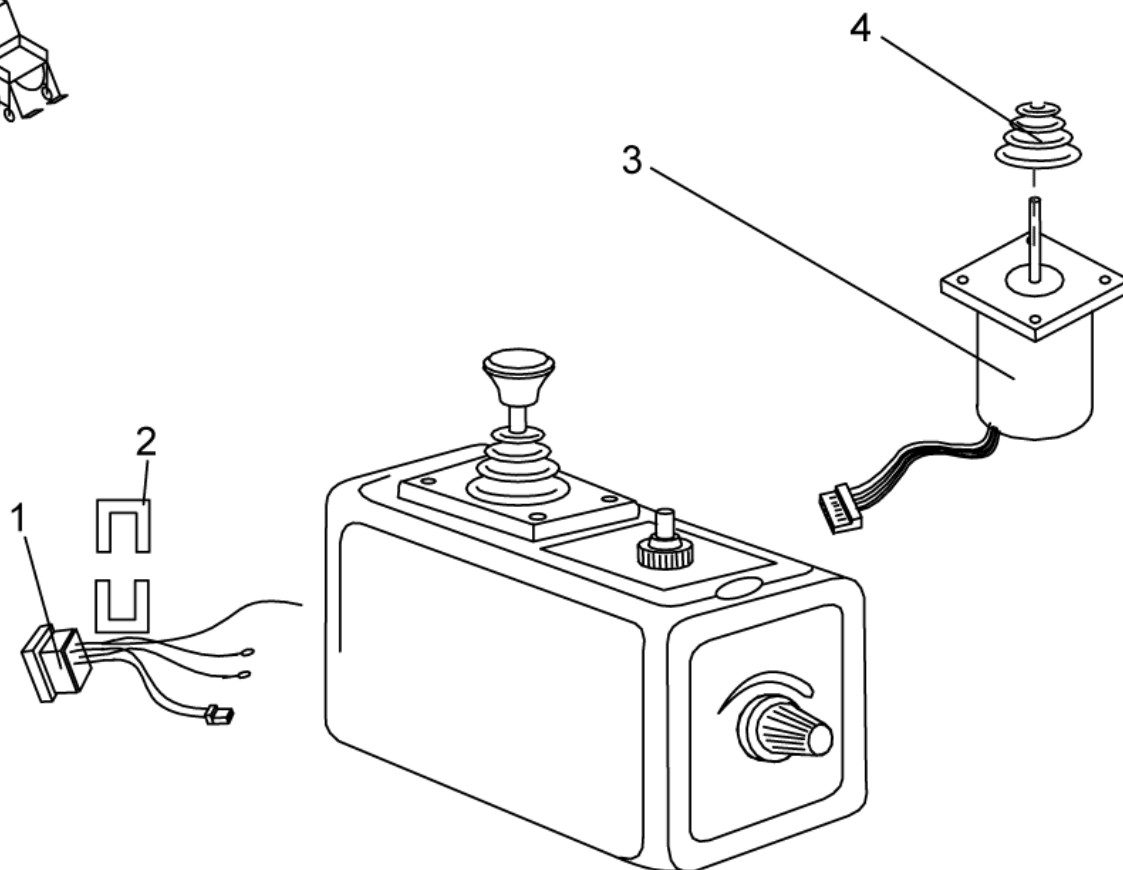6001425.dwg




item	ordercode	description	quantity
1	P41105	BUS CABLE SOCKET	1
2	GME61927	FIXING CLAMP	1
3	P10304	JOYSTICK	1
4	Z183001A	RUBBER GAITER; BLACK	1

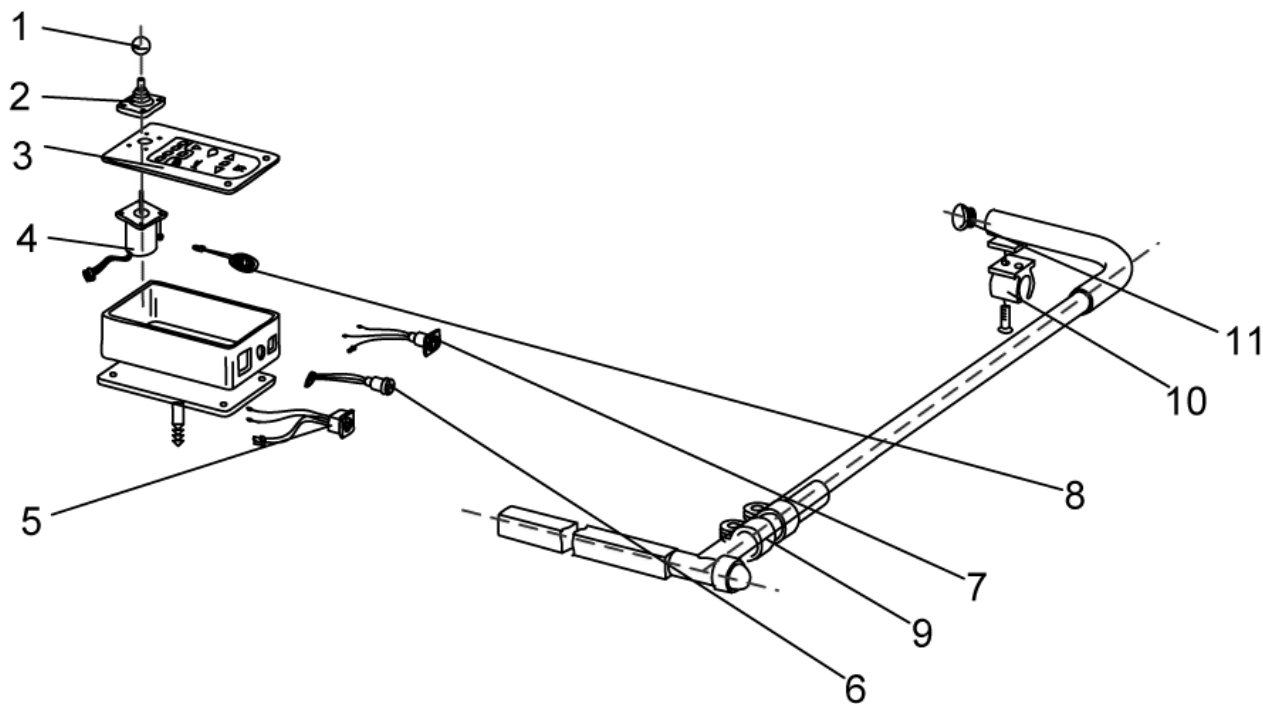

item	ordercode	description	quantity
1,2,3,4,5	P10066	ACS-TRAY CONTROL BOX	1
1	L36999	KNOB 32 MM, HOLE 5,8 MM	1
2	Z183001A	RUBBER GAITER; BLACK	1
3	P30907	FOIL KEYPAD	1
4	GSM55120	ACS-JOYSTICK F. TRAYCONTROL	1
5	P41108	ACS-BUS CABLE SOCKET	1
6	P41107	PROGRAMMER SOCKET	1
7	P41106	CHARGER SOCKET	1
8	P30493	HORN	1
9	5006720	CLAMP 22 MM	2
10	Z16001A	CLAMP	1
11	L01008	TUBE END PLUG SL 22	2

Chin Control Assembly Kit

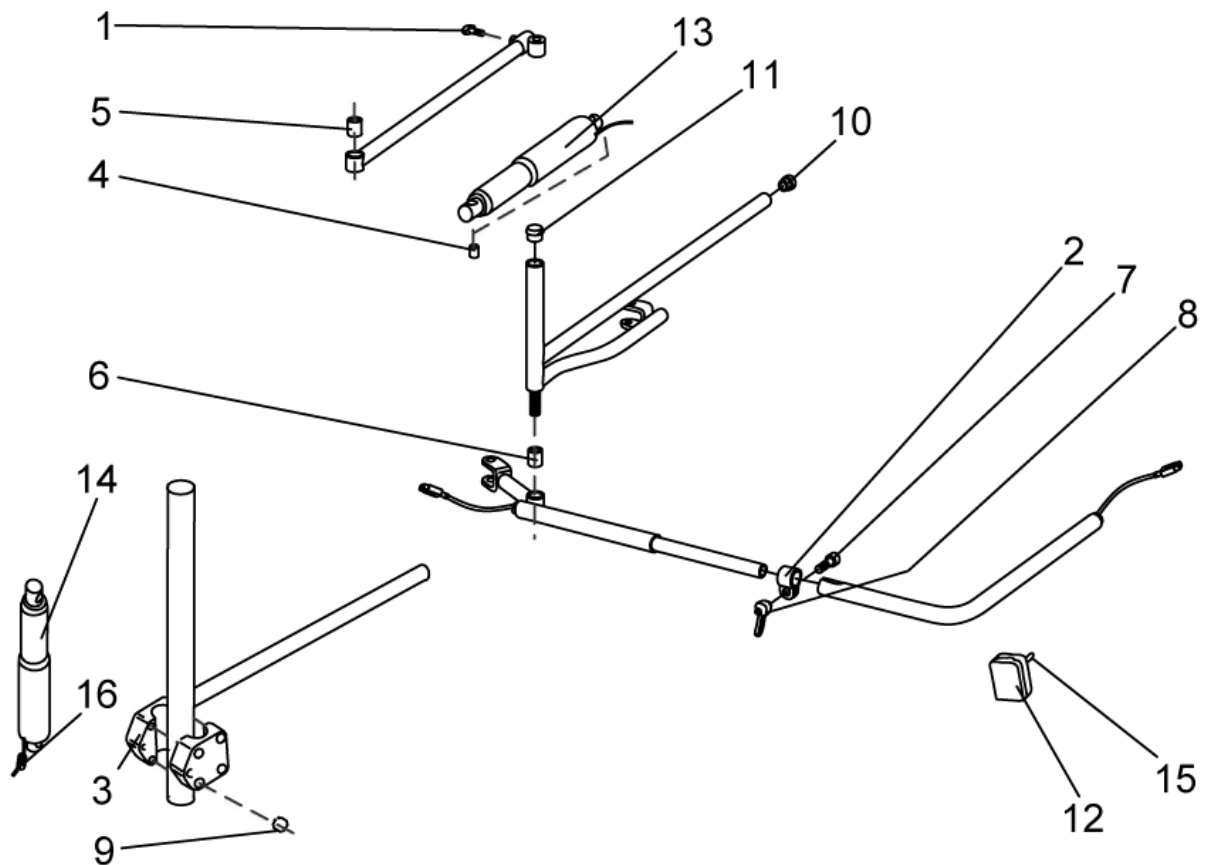F3_Kinn_Abschwenk.dwg



item	ordercode	description	quantity
1	L31001	HAND KNOB M6 X 10	1
2	5006720	CLAMP 22 MM	1
3	Z12009A	CLAMP	2
4	4000665-0002	SPACER BUSH 10X1X22,5	2
5	A17012A	BUSH RD. 18 X 22	1
6	Z131009A	BUSH RD. 18 X 3 X 22	1
7	E30101	CUP HEAD NIBBED BOLT M8 X 25	1
-	L01041	NUT CAP M8	2
-	L01037	NUT CAP M12	2
8	L32901	LEVER M8; BLACK	1
9	K22103	CAP A10	1
10	L01049	TUBE END PLUG SL 18	2
11	L01008	TUBE END PLUG SL 22	4
12	L90994	MULTIBOX SMALL	1
13	P00082	ACTUATOR LA 22.5E-75-24 INCL CABLE AND SWITCH	1
14	P00081	ACTUATOR LA 22.5E-100-24 WITH STOP-SWI	1
-	P12194	POLYSWITCH FUSE RXE 135	1
15	P30109	TOGGLE SWITCH SINGLE POLE	1
15	Z183011A	TOGGLE SWITCH EXTENSION; GREY	1
16	P09596	FERRITE SLEEVE C=8 D=1	1


Chin Control Dash Board

Diagram illustrating the assembly of a wheelchair seat component, showing the relationship between various parts:

- 1: Wheelchair frame (indicated by a small icon in the top left corner).
- 2: Seat cushion.
- 3: Seat base plate.
- 4: Mounting bracket.
- 5: Seat backrest.
- 6: Seat base plate (bottom view).
- 7: Mounting bracket (bottom view).
- 8: Small circular fastener (bottom view).
- 9: Small cylindrical fastener (bottom view).
- 10: Mounting bracket (top view).


item	ordercode	description	quantity
1	Z183265	FOAM CONTROL KNOB CPL.	1
2	Z183001A	RUBBER GAITER; BLACK	1
3	P10304	JOYSTICK	1
4	Z183229	JOYSTICK KNOB CPL.	2
5	F383019	UPPER HOUSING; GREY	1
6	Z183242	LOWER HOUSING LH	1
6	Z183241	LOWER HOUSING RH	1
7	P41106	CHARGER SOCKET	1
8	T95099	O-RING 14-3 MM	1
9	P41104	PROGRAMMER SOCKET	1
9	P41108	ACS-BUS CABLE SOCKET	1
9	P41106	CHARGER SOCKET	1
10	Z183243	ACS-CHIN CONTROL JOYSTICK SWITCH LH	1
10	Z183244	ACS-CHIN CONTROL JOYSTICK SWITCH RH	1
-	P10475	STICKERSET (ACS CHIN CONTROL)	1
-	P09594	FERRITE SLEEVE C=6.4,D=2 TDK ZCAT 1325-0530A	1
-	P30499	HORN (DIGISOUND)	1
-	Z183048	HORN CLAMP	1
-	GME61927	FIXING CLAMP	2

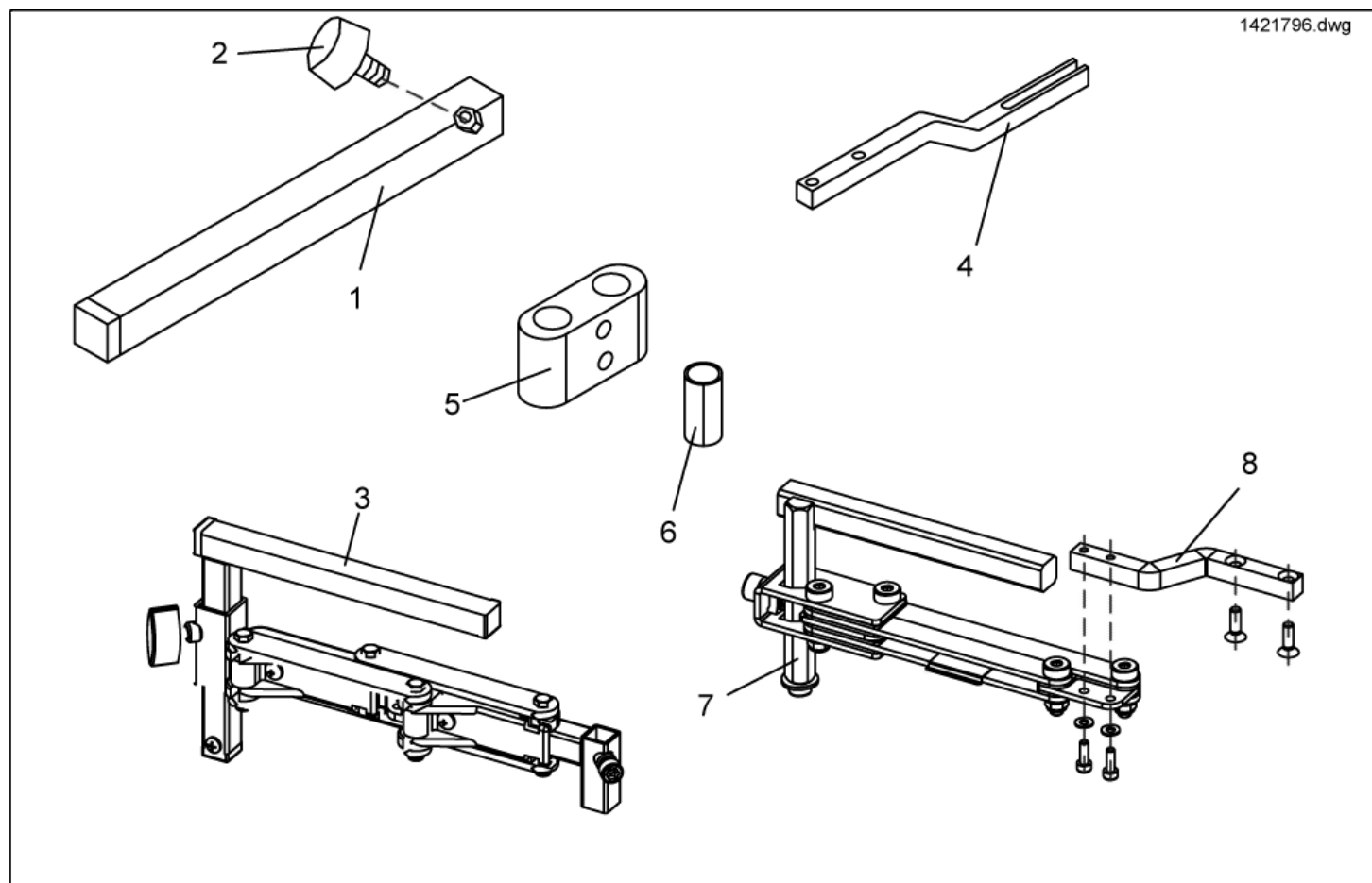Remote fixation, standard/ swing away









item	ordercode	description	quantity
1	SPS183040	REMOTE HOLDER STANDARD	1
2	L31001	HAND KNOB M6 X 10	1
3	SP1421796	SWING-OUT ACS REMOTE HOLDER LH	1
3	SP1421795	SWING-OUT ACS REMOTE HOLDER RH	1
4	SP1440493	JOYSTICK BOX SUPPORT SWING-OUT REM 24SD AJ, WITH SCREWS	1
5	1440631	TUBE CLAMP DIA 22 MM; BLACK	1
6	Z183264.99	BUSH RD. 22 X 1,5 X 40	4
-		MORE CONTROLL UNIT SUPPORTS »»» SEE "CONTROL UNIT SUPPORT TABULAR"	1
7	1496248	SWING AWAY MECHANISM RH, CPL.	1
7	1496249	SWING AWAY MECHANISM LH, CPL.	1
8	1498222	INSTALLATION KIT REMOTE HOLDER SWING AWAY MECHANISM FOR REM24 SD	1

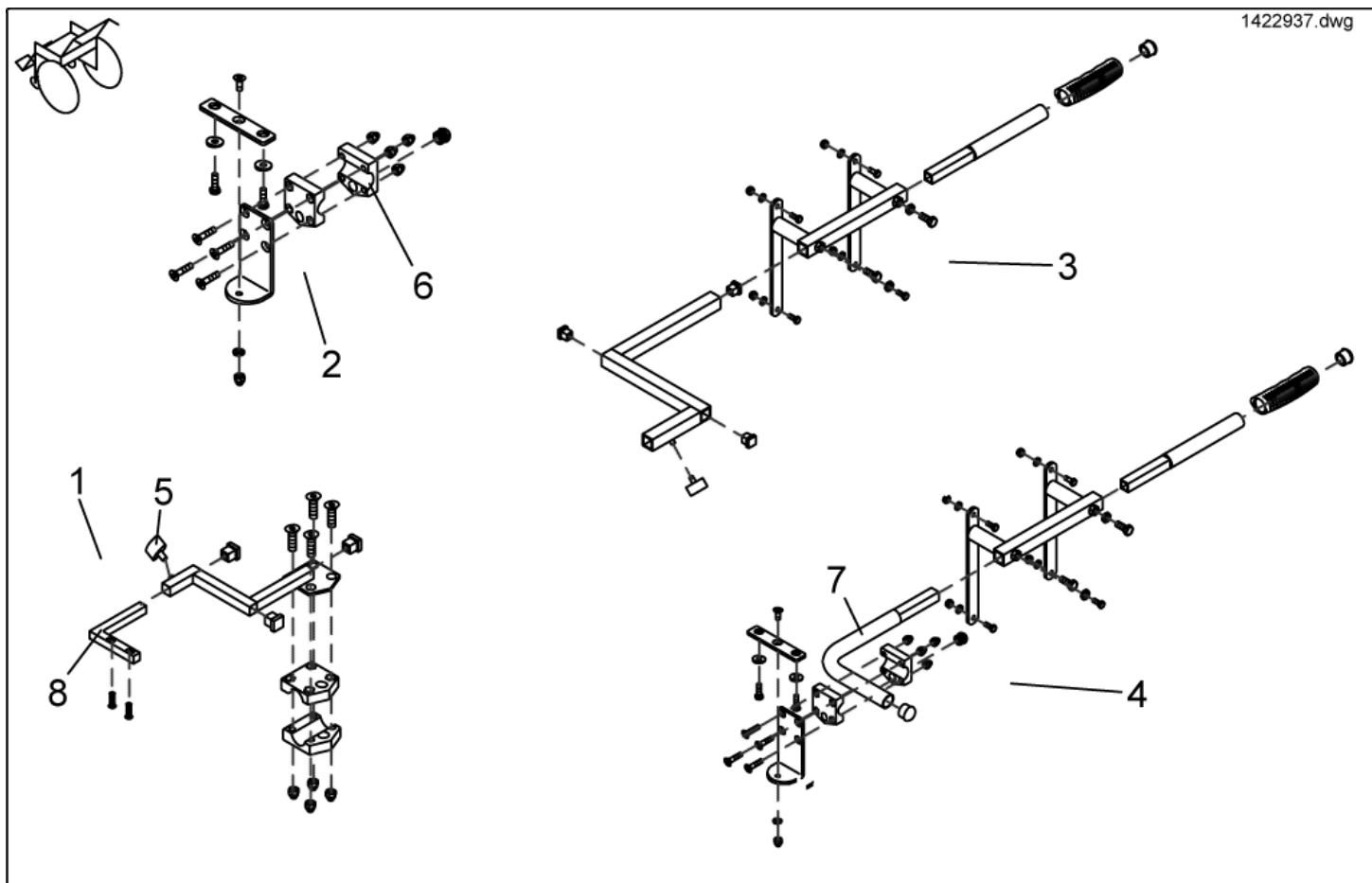Remote fixation, for Attendant Control




item	ordercode	description	quantity
1	SP1422937	ASSEMBLY KIT F. STD. CARER SEAT CONTROL LH .	1
1	SP1422936	ASSEMBLY KIT RH F. STD. CARER SEAT CONTROL	1
2	SP1422818	ASSEMBLY KIT F. SWITCHABLE CARER CONTROL CPL.	1
3	SP1422814	ASSEMBLY KIT F. SWITCHABLE CARER CONTROL CPL., RECARO LH	1
3	SP1422813	ASSEMBLY KIT F. SWITCHABLE CARER CONTROL CPL., RECARO RH	1
4	SP1422819	ASSEMBLY KIT F. SWITCHABLE CARER RECARO CONTROL	1
5	L31001	HAND KNOB M6 X 10	1
6	Z12008A.P40	GRIPPING CLIP W. HEXAGON HOLE	2
7	F383065-9005	HOLDER FOR CARER RECARO CONTROL; RAL 9005, DEEP BLACK	1
8	SP1440289	ATTENDANT JOYSTICK BOX SUPPORT REM 24SD WITH SCREWS, LEFT	1
8	SP1440288	ATTENDANT JOYSTICK BOX SUPPORT REM 24SD WITH SCREWS, RIGHT	1
-	NA	MORE CONTROLL UNIT SUPPORTS »»» SEE "CONTROL UNIT SUPPORT TABULAR"	1

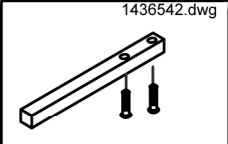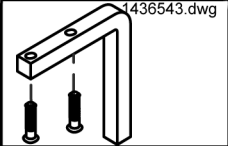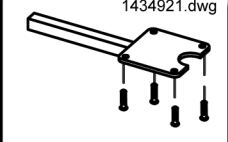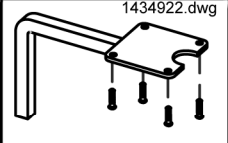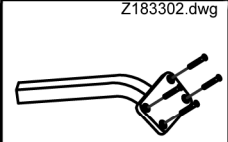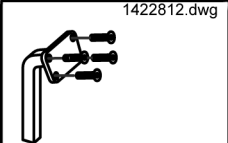Control Unit support tabular


Picture	joystick box	Standard	swing out	ordercode
	REM 24 SD	X		SP1436542
	REM 24 SD		X	SP1436543
	REM 34	X		SP1434921
	REM 34		X	SP1434922
	G80i	X		SPZ183302
	G80i		X	SP1422812

Desk/ Tray control

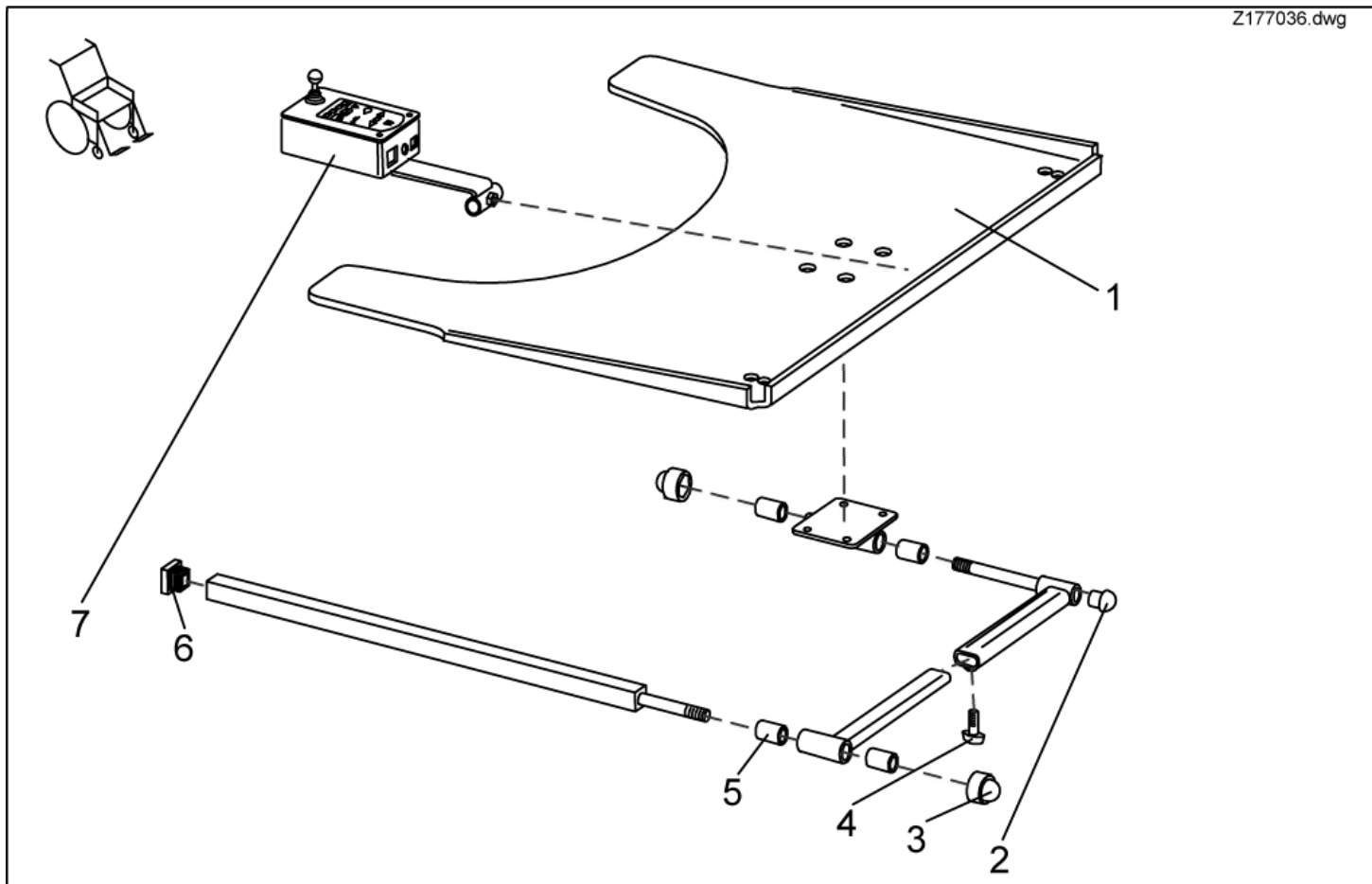Z177036.dwg




item	ordercode	description	quantity
1,2,3,4,5,6	SP1422952	SWING-OUT TRAY F. CONTROL LH CPL.	1
1,2,3,4,5,6	SP1422953	SWING-OUT TRAY F. CONTROL RH CPL.	1
2,3,4,5,6	SPZ177011	TRAY SUPPORT FIXING KIT CPL.	1
1	Z177017	TRAY-LH CONTROL; TRANSPARENT	1
1	Z177016	TRAY-RH CONTROL; TRANSPARENT	1
2	L01008	TUBE END PLUG SL 22	1
3	L01037	NUT CAP M12	2
4	L31001	HAND KNOB M6 X 10	1
5	Z17032A	BUSH RD. 19 X 3.5 X 25	4
6	L01055	TUBE END PLUG E 15 X 15	2
7	SP1422798	TRAY CONTROL CPL.	1

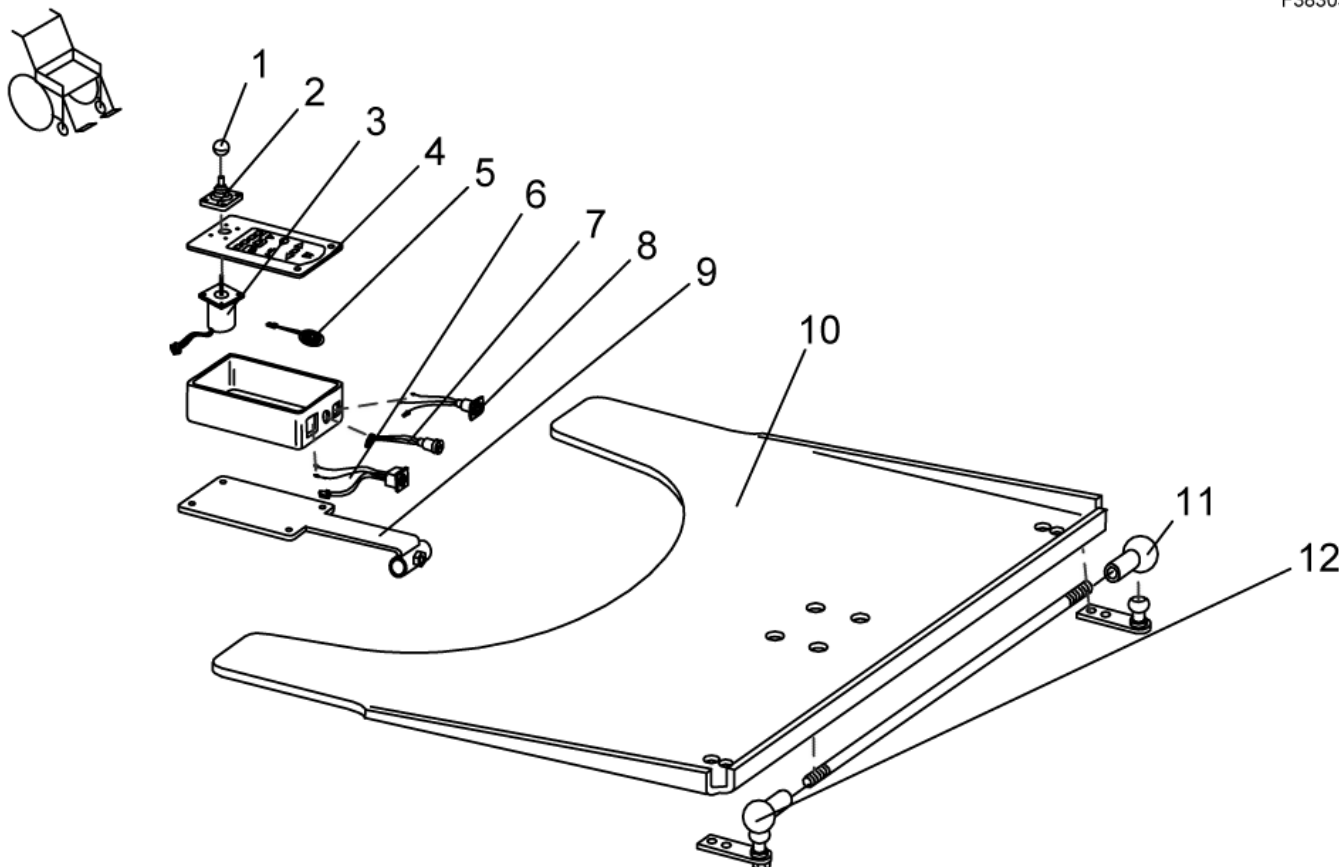

item	ordercode	description	quantity
-	P10066	ACS-TRAY CONTROL BOX	1
1	L36999	KNOB 32 MM, HOLE 5,8 MM	1
2	Z183001A	RUBBER GAITER; BLACK	1
3	GSM55120	ACS-JOYSTICK F. TRAYCONTROL	1
4	P30907	FOIL KEYPAD	1
5	P30493	HORN	1
6	P41108	ACS-BUS CABLE SOCKET	1
7	P41107	PROGRAMMER SOCKET	1
8	P41106	CHARGER SOCKET	1
9	L60999	FOOT M4 X 13	2
10	Z17024A	TRAY 660 X 540 X 8; TRANSPARENT	1
11	N04109	SWIVEL JOINT AS 13-M8 RH	2
12	SP4100136	SUPPORT ROD ; CPL..	1

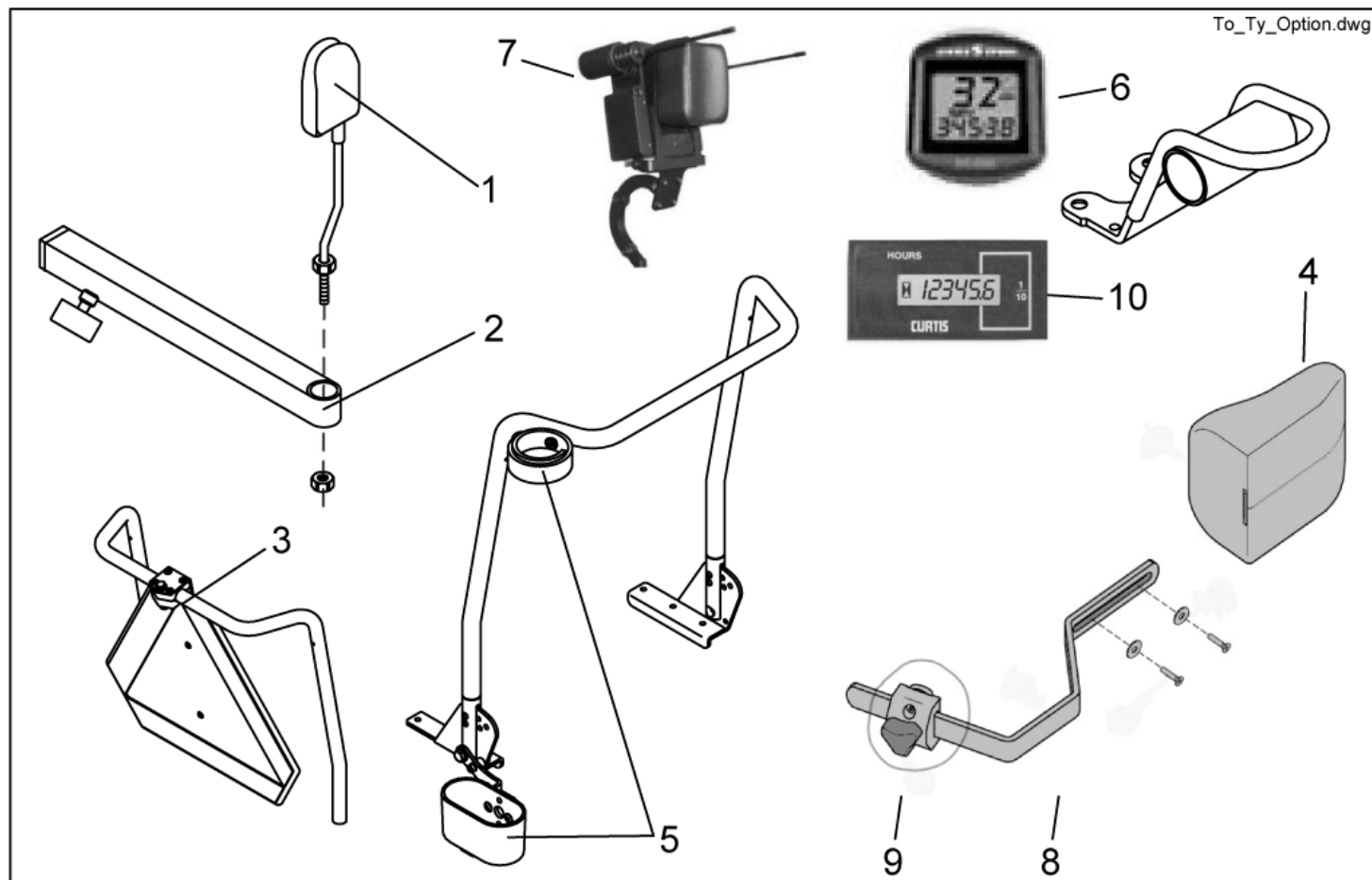Options



item	ordercode	description	quantity
-	Z12008A.P40	GRIPPING CLIP W. HEXAGON HOLE	2
-	Z183255-9005	ACS-SPEEDOMETER BRACKET REM 24; RAL 9005	1
-	Z183296-9005	ACS SPEEDOMETER HOLDER G 80I; RAL 7021BLACK-GREY	1
-	Z183259	ACS-SPEEDOMETER CABLE	1
-	1421733	ACS-REED-SENSOR CABLE	1
-	P22317	CABLE SOCKET 6 X 10 X 13; BLACK	1
-	P30497	MAGNET, ROUND 6 X 25	1
-	SP1422868	ON-BOARD TOOLS C/W BAG, REPLACEMENT SET	1
-	Q99991	PUMP WITH INSERT HOSE	1
-	1440530	TRANSPORT RAMP	1
1	SP1422955	REAR MIRROR	1
2	SP1423435	REAR MIRROR SUPPORT RAIL CPL.	1
1,2	SP1422881	REAR MIRROR COMPLETELY WITH SUPPORT	1
3	SP1430410	REAR MARKING SIGN ECE-R 69 CPL.	1
-	1430834	REAR MARKING SIGN ECE-R 69	1
4	1422545	TRUNK SUPPORT FLEX; BLUE	2
4	1440284	TRUNK SUPPORT, BLACK	2
4	5253067-TR04	TRUNK SUPPORT; VINYL GREY	2
-	5253170	FIXING PIECE	2
5	1442701	CANE HOLDER FOR STANDARD-SEAT, COMPLETE	1
5	1442702	CANE HOLDER FOR CONTOUR/FLEX-SEAT, COMPLETE	1
5	1442725	CANE HOLDER FOR RECARO-SEAT, COMPLETE	1
6	SP1421843	SIGMA SPORT BC 600 SPEEDOMETER; CPL. (REM 24)	1
6	SP1422841	SIGMA SPORT BC 600 SPEEDOMETER; CPL. (G 80I)	1
-	1442684	TOUCH-UP PAINT RAL 9005 BLACK	1
-	Z176015A	NYLONBAG	1
7	1442679	HEAD CONTROL UNIT ACS	1

item	ordercode	description	quantity
8	1489009	ASSEMBLY KIT FOR PELOTTE UPHOLSTERY	1
8	1423700-9005	HANDLE FOR PELOTTE	1
9	5253170	PIECE OF ATTACHMENT FOR PELOTTE	1
10	SP1485698	HOUR COUNTER FOR DC-MOTOR	1
10	SP1485700	HOUR COUNTER FOR GB-MOTOR	1

Documentation - User Manuals Typhoon

- You will find documentation for downloading on our Internet pages.

deutsch

english

français

netherlands

español

portugues

suomi

svenska

norsk

dansk

italiano

item	ordercode	description	quantity
1	1435056	USER MANUAL Typhoon "GERMAN"	1
2	1435058	USER MANUAL Typhoon "ENGLISH"	1
3	1435057	USER MANUAL Typhoon "FRENCH"	1
4	1435062	USER MANUAL Typhoon "DUTCH"	1
5	1435063	USER MANUAL Typhoon "SPANISH"	1
6	NA	USER MANUAL Typhoon "PORTUGUESE"	1
7	1436381	USER MANUAL Typhoon "ITALIAN"	1
8	1436382	USER MANUAL Typhoon "FINNISH"	1
9	1435059	USER MANUAL Typhoon "SWEDISH"	1
10	NA	USER MANUAL Typhoon "NORWEGIAN"	1
11	1435060	USER MANUAL Typhoon "DANISH"	1

deutsch

english

français

netherlands

español

portugues

suomi

svenska

norsk

dansk

italiano

item	ordercode	description	quantity
1	1436142	USER MANUAL REM 24 SD "GERMAN"	1
2	1436144	USER MANUAL REM 24 SD "ENGLISH"	1
3	1436143	USER MANUAL REM 24 SD "FRENCH"	1
4	1436146	USER MANUAL REM 24 SD "DUTCH"	1
5	1436148	USER MANUAL REM 24 SD "SPANISH"	1
6	1436386	USER MANUAL REM 24 SD "PORTUGUESE"	1
7	1436385	USER MANUAL REM 24 SD "ITALIAN"	1
8	1436147	USER MANUAL REM 24 SD "FINNISH"	1
9	1436150	USER MANUAL REM 24 SD "SWEDISH"	1
10	1436149	USER MANUAL REM 24 SD "NORWEGIAN"	1
11	1436145	USER MANUAL REM 24 SD "DANISH"	1

Documentation - User Manuals miscellaneous

- You will find documentation for downloading on our Internet pages.

deutsch

english

français

netherlands

español

portugues

suomi

svenska

norsk

dansk

italiano

item	ordercode	description	quantity
1	Q99083	USER MANUAL ACS-CHINCONTROL "GERMAN"	1
1	1419316	USER MANUAL HANDHELDPROGRAMMER "GERMAN"	1
1	1419317	USER MANUAL HANDHELDPROGRAMMER "ENGLISH"	1
1	Q99081	OPERATION INSTRUCTION, TRAY-/CENTRAL STEERING	1
2	Q99086	USER MANUAL ACS-CHINCONTROL "ENGLISH"	1
3	Q99082	USER MANUAL ACS-CHINCONTROL "FRENCH"	1