svenska

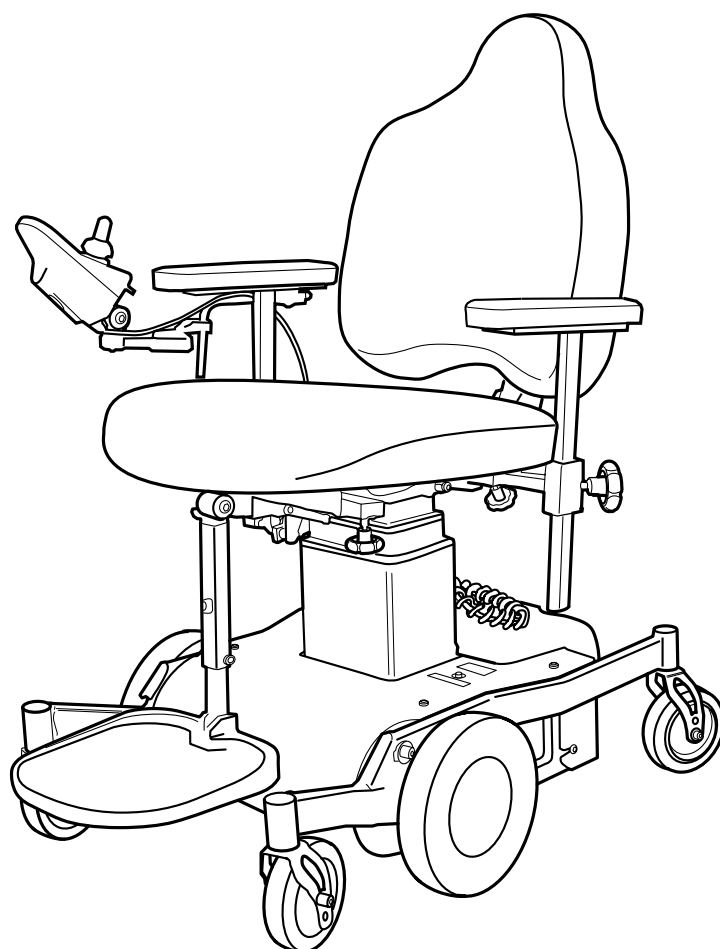
english

bruksanvisning / manual

EUROFLEX

flexmobil i6 SitRite

Eurovema

Creates mobility and ergonomics.

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For you with visual impairment:
This manual is available in pdf format with
zoom at www.eurovema.se
see documentation - manuals.

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english

INTRODUCTION

We congratulate you to the choice of Flexmobil i6 as your power wheelchair!

We will ensure that you will be satisfied with this Euroflex product from product from Eurovema Mobility AB, constructed and build in Sweden. Flexmobile i6 is built to meet very high standards of ergonomics, seating comfort and function.

Read the instructions carefully to benefit i6 all possibilities.

USERS AND USE

Flexmobile i6 is an electric indoor wheelchair. It is intended for children and adults with limited mobility and can be used by the user, assistant or both. Flexmobil i6 is available with different seating sizes that have a variety of settings and are easy to adapt individually for every need. Expected design life:10 years.

TEST SPECIFICATION

Flexmobile i6 is the CE mark and meet the requirements of Law 1993: 358 (93/42 / EEC) on medical devices and the MPA procedure for such medical devices, LVFS 2013: 11. It meets all the requirements of EN 12184:2014 Class A.

CONTACT

If you have questions or need help with your product, please contact the local agent. Contact information manufacturer:
Eurovema Mobility AB
Tel:+46-371-390100
E-mail: info@eurovema.se
Baldersvägen 38
332 35 GISLAVED
SWEDEN

ASSOCIATED DOCUMENTS

Associated product data sheets, service manual, installation instructions, safety notices and possible product recalls are available to read or download at www.eurovema.se

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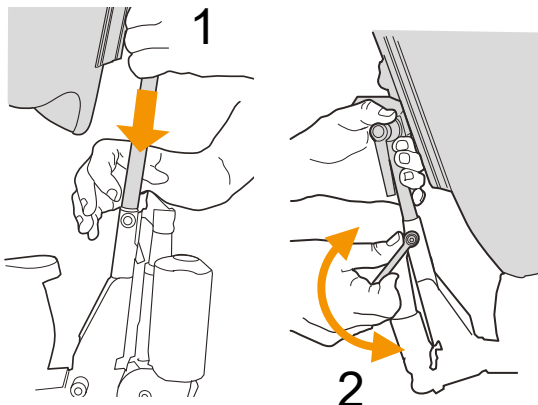
english

ASSEMBLY

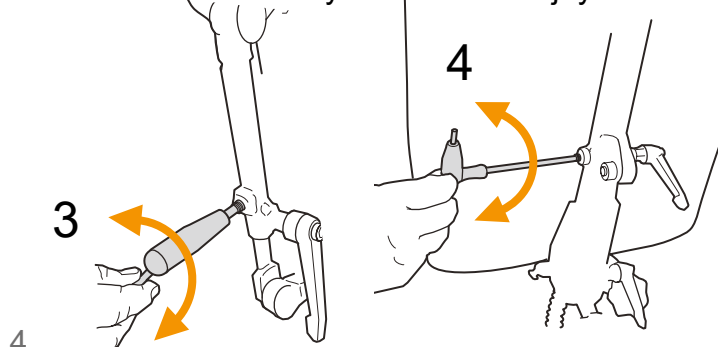
- Open the packing and check that no transport damage has occurred.
- Also check that the delivery harmonize with the order.
- If the chair is delivered with backrest and armrest not mounted.
- Press in the spring ball and slide in the back support pipe as shown in the picture (1). Tighten the handle (2) and the safety screw on the side of the back support pipe (3, 4).

TRANSPORTATION

When transporting Flexmobil in a motor vehicle it is important that the brakes are engaged. See section "Disengagement of brakes". The chair should be secured with tie down straps. Attachment loops are available as standard. Sitting in the wheelchair during the transport is strictly forbidden. Reduce the transport size by removing the armrest, backrest and the leg rest. The batteries in the wheelchair are sealed and maintenance free type AMG and approved for transport in aircraft's. Further on transport see page 34.

Check before use that,

- the wheels are correct bolted.
- all knobs and screws are tightened.
- the freewheel device for the brakes are deactivated.
- no cables are pinched or damaged.
- the arm- and backrests are raised and do not touch the hood seats lowest position.
- the display shows no error codes(no flashing)
- battery is fully charged.
- the joystick steering works in all directions.
- brakes works when you release the joystick.

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english

SAFETY

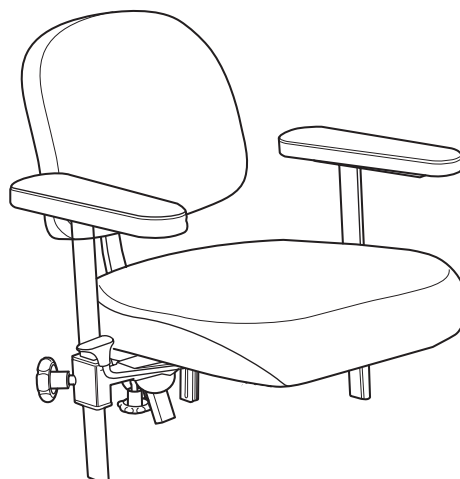
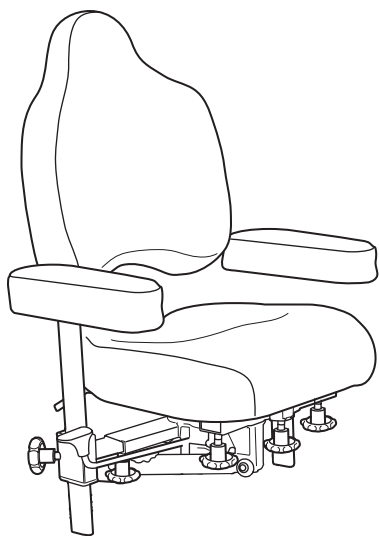
- Carefully read the manual before using the wheelchair.
- Charge the batteries immediately when battery indicator shows red.
- The lift motor has a duty cycle of 2 on /18 off meaning 2 minutes use followed by 18 minutes of rest.
- The wheelchair is designed to be used on in normal indoor climate.
- Be careful when adjusting the manual seat angle when seated. Could cause a risk of falling out of the chair.
- Immediately contact service if damages, i.e. loose parts or change in function of the chair is discovered.
- Use of the different seat settings can affect stability of the wheelchair. Use these only when the wheelchair is on a flat surface.
- Make sure to tighten all screws, knobs and levers after adjustments.
- Metal surfaces can become very hot if they have been exposed to sunshine or radiated heat.
- Service, maintenance and adaptations should be performed by educated staff, authorized of Eurovema Mobility AB.
- Use only original parts from Eurovema Mobility at repair.
- Do not exceed maximum user weight 150kg.
- Use only original charger for charging of the batteries.
- For maintaining safety use only original fasteners to the removable parts i.e. Armrest, backrest and foot-plate.
- The wheelchairs function may be disturbed in strong electromagnetic fields radiated by the e.g. power lines and data centers. The wheelchair itself can disturb the equipment based on electromagnetic fields such as alarm systems in shop, automatic doors, etc.

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english

Backrest combinations SittRite, Comfort and
Childseat ABC

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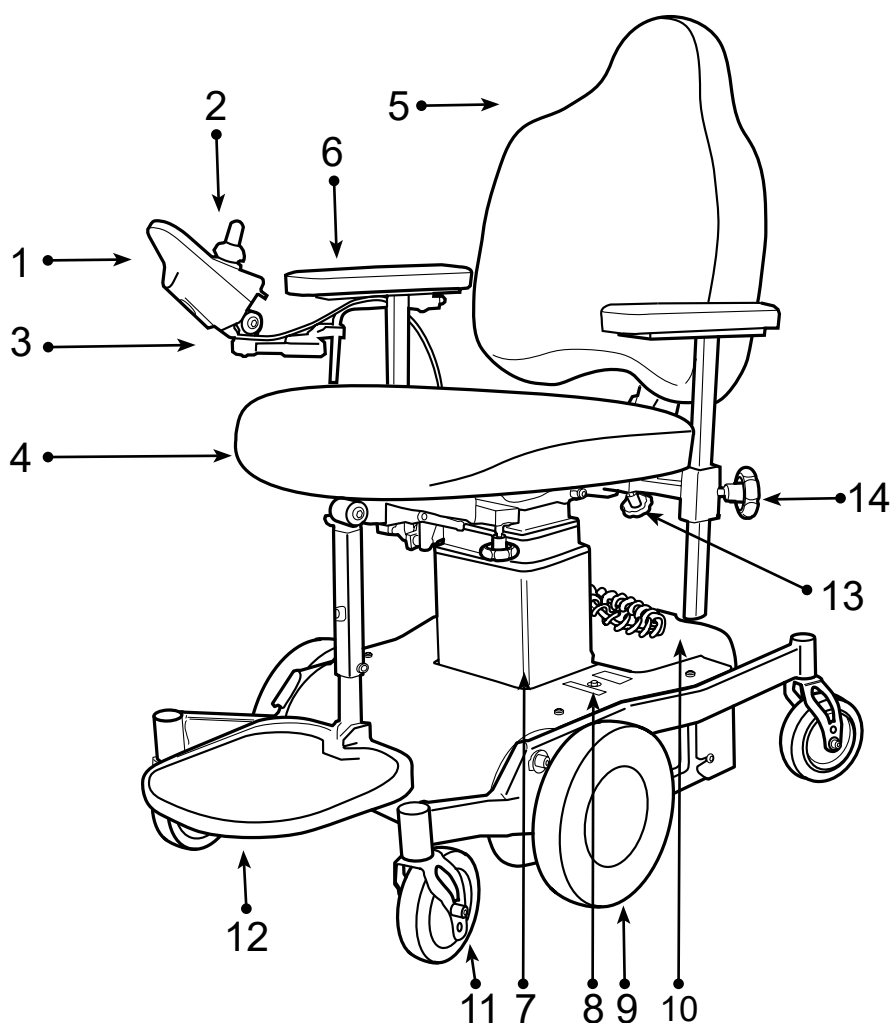
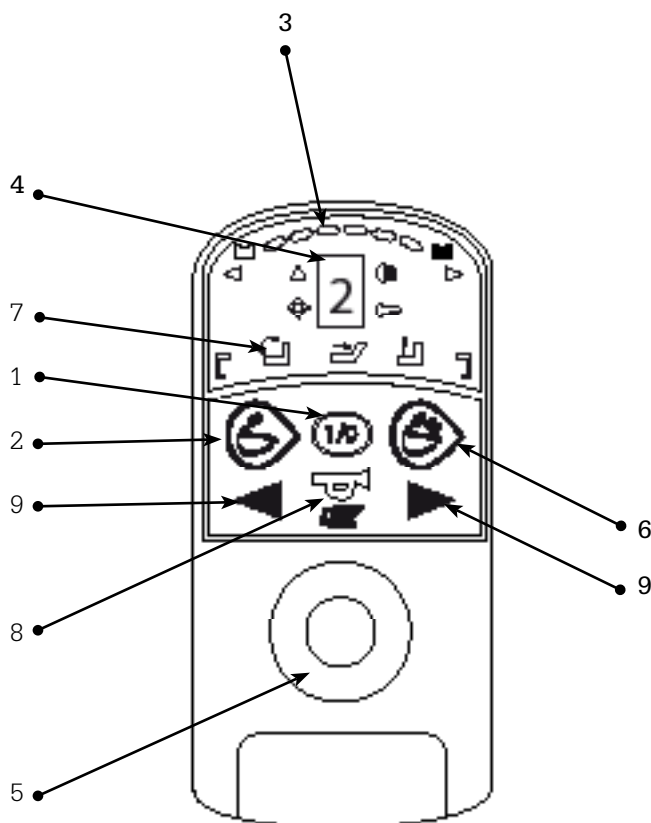
english

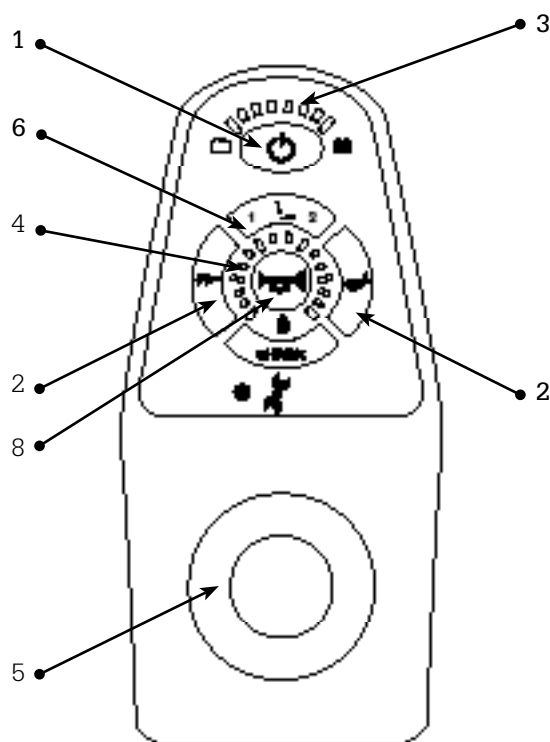
ELECTRONICS

The Flexmobil i6 is equipped with Dynamic DX control system to control the power from the batteries out to the motors.

The wheelchair and it's power assisted seat functions can be maneuver by the control box, available in two different models; G90T och SHARK.

The joystick is for driving the wheel chair in the desired direction. The electronics can be programmed to suit the users needs but usually the original program is fully sufficient. Should a fault occurs with the electronics the status LED (8) will blink. Thru counting the number of blinks you can deduce the problem. See chap. fault trace.

- 1) on/off
- 2) choice of drive programs
- 3) battery indicator
- 4) display for drive programs/speed
- 5) joystick
- 6) seat functions
- 7) symbols for seat functions
- 8) status LED
- 9) horn

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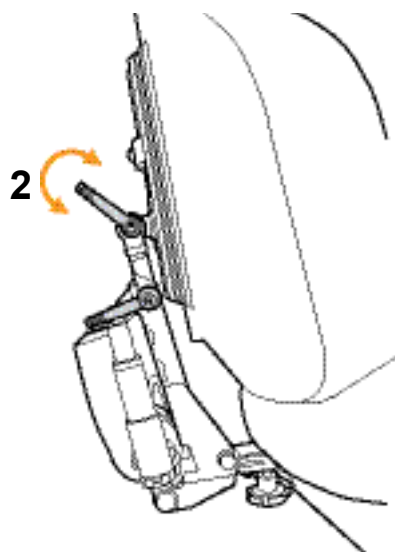
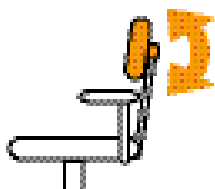
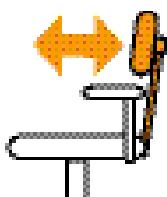
english

BACK REST - seat depth

To adjust the seat depth move the back support forward or backwards. Loosen the knob (1). Adjust the desired seat depth by moving the back support forward or backwards. Lock by tighten the knob.

BACK REST - angle

To adjust the angle on the back support, loosen the lever (2). Adjust to desired angle and lock by moving the lever back to its original position.

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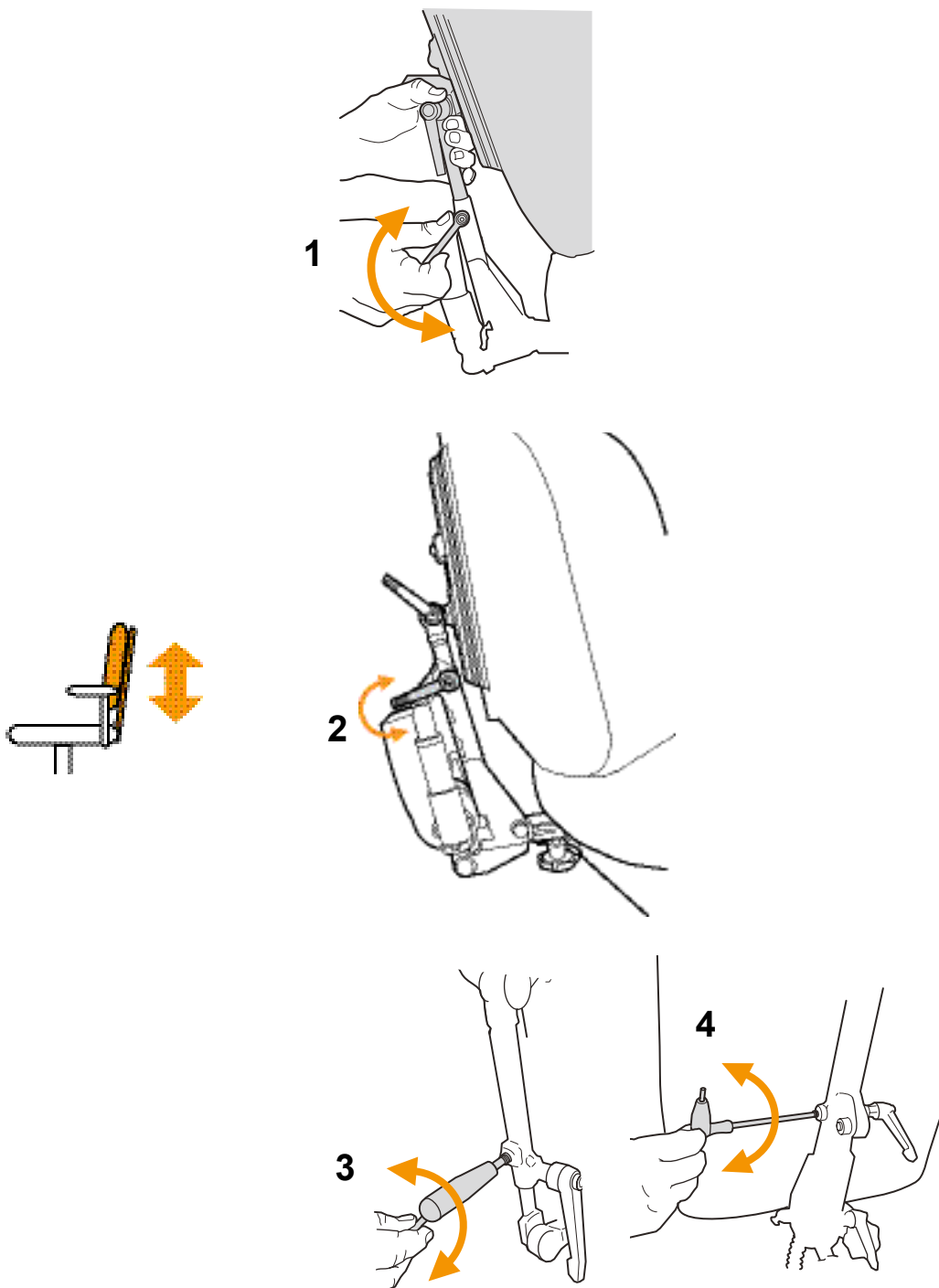
english

BACK REST - height

Loosen the security screw with an allen key before adjusting the high of the backrest (1).

Loosen the lever (2) by turning it anticlockwise $\frac{1}{2}$ turn. Adjust to desired height of the back support and lock turning the lever clockwise $\frac{1}{2}$ turn.


After adjustment tighten the security screws.

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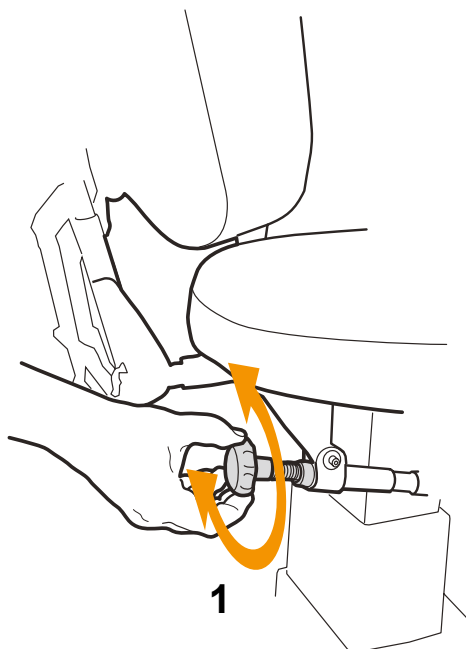english

SEAT ANGLE - manual

The seat angle is possible to adjust from -14° to $+32^{\circ}$ backward. Turn the knob clockwise to tilt the seat forward and counterclockwise for backward (1).

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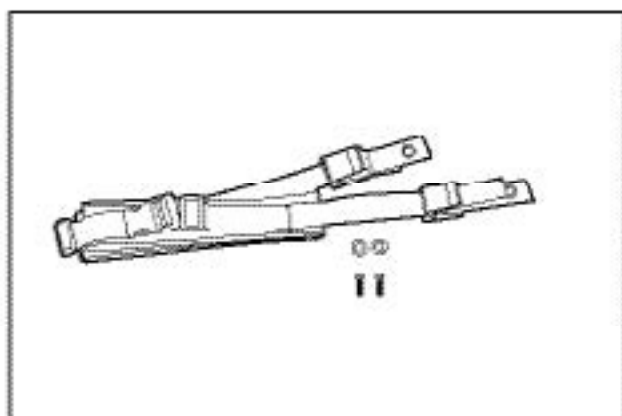
Additional pelvic support

NOTE! For correct support for the pelvis and stability to the torso, we recommend using seating system SitRite together with hip belt and possible thigh support.






The has an ergonomic built-in position cushion that prevents sliding and provides optimal pelvic positioning. The pressure moves from the pelvis to the thighs. This results in lower pressure on vulnerable areas.

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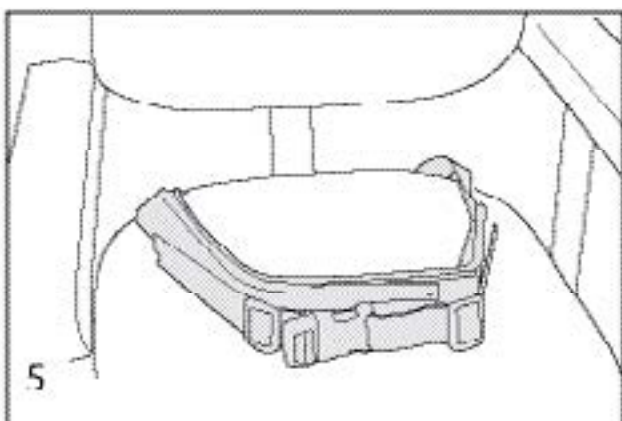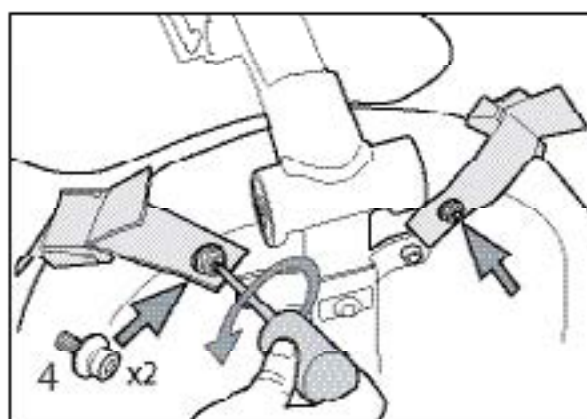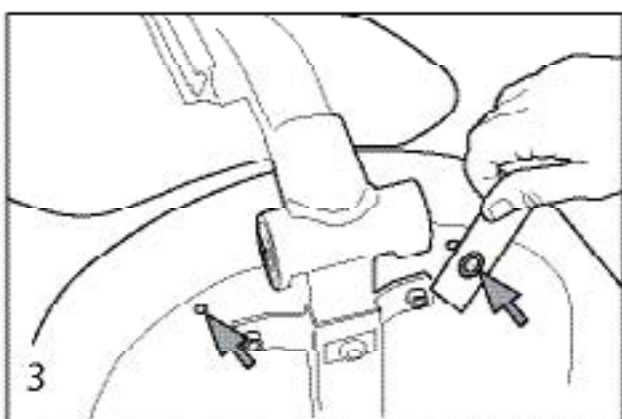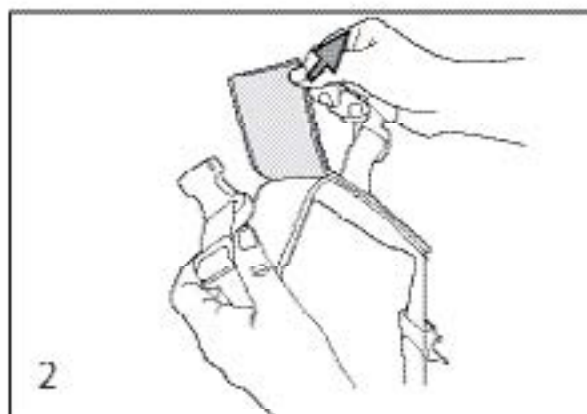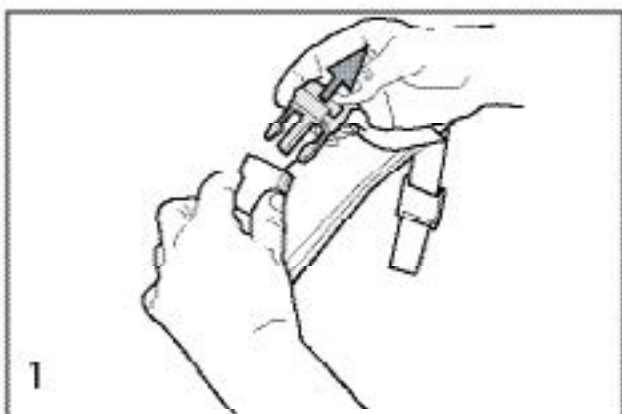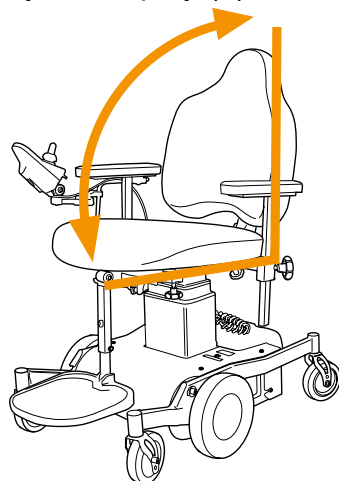
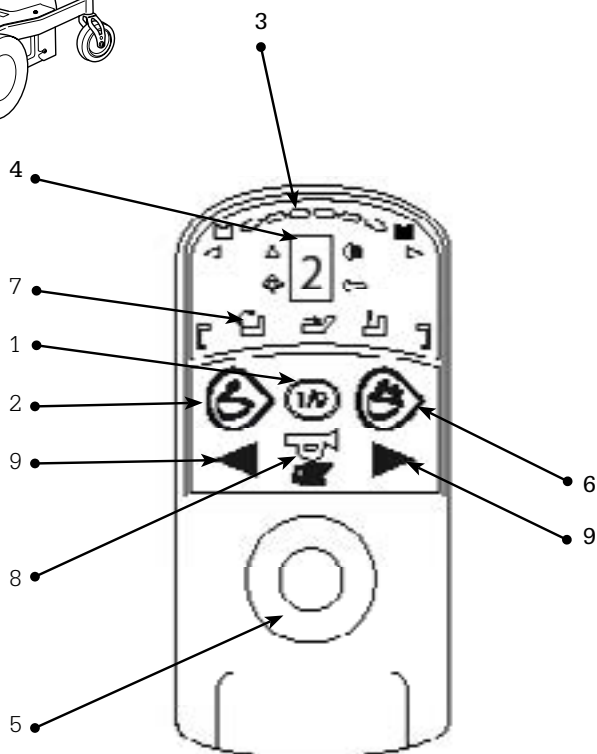
english

SEAT ANGLE - power

If the chair is equipped with power assisted seat tilt it can be activated by the controlbox. Push button "6".

G90T

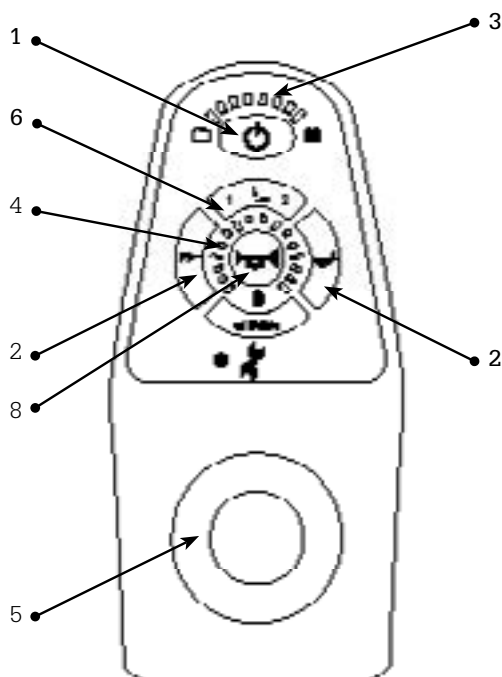
Choose a seat symbol (7) for the seat angle by moving the joystick (5) to one side to the second icon from the left side blinks.

SHARK

Choose a seat symbol (6). Press the seat symbol (6) and select the number "2".

Move the joystick forward to activate the seat tilt forward and backwards for backwards.

The seat tilt stops automatically when the end points is reached. Push button (2) to get back into drive mode and the choosed drive program will be visble in the display (4).

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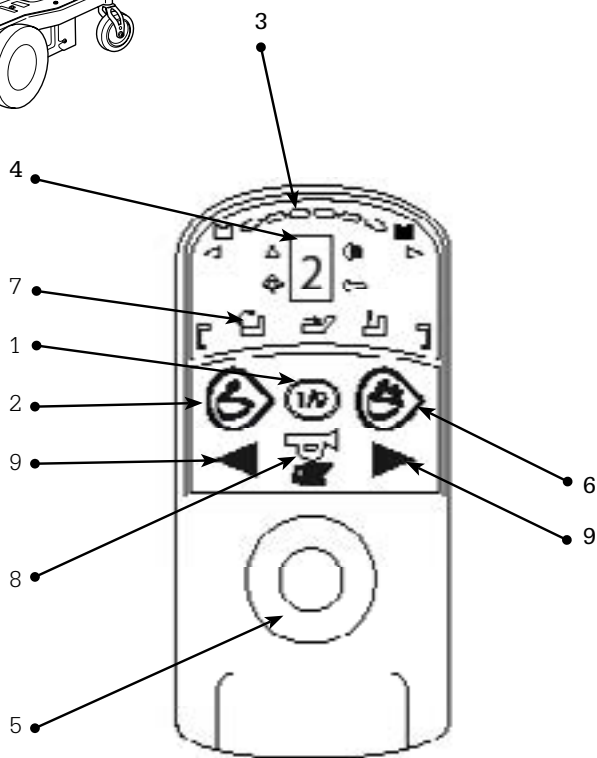
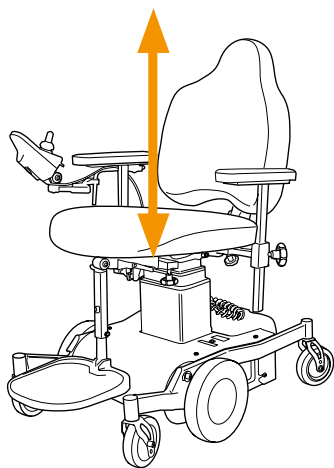
english

SEAT HEIGHT

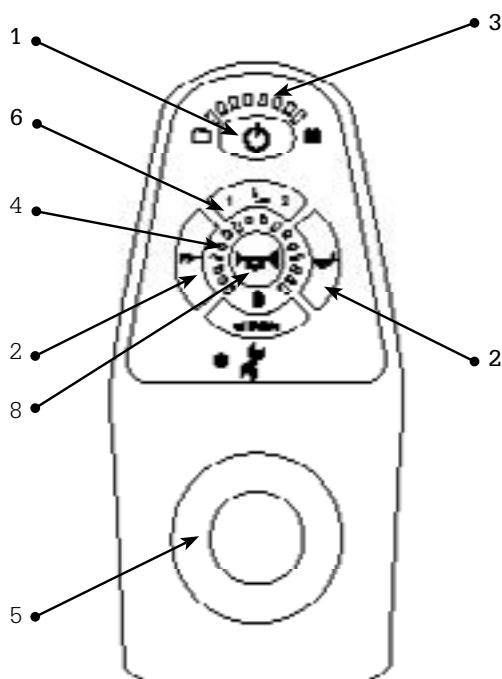
Start the electronics of the wheel chair by pushing button (1) on the control box. Then push button (6).

G90T

Choose a seat symbol "7" and pull the joystick to the side to select the fourth icon from the left. For seat lift thro leaning the joystick (5) side ways until the symbol blink.

Shark

The nubmer "1" lights up on button (6). Move the joystick forward to activate the seatlift up and backwards down. The seat lift stops automatically when the end points is reached. Push button (2) to get back into drive mode and the choosed drive program. It will be visble in the display (4).

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english

ARM REST - height and width

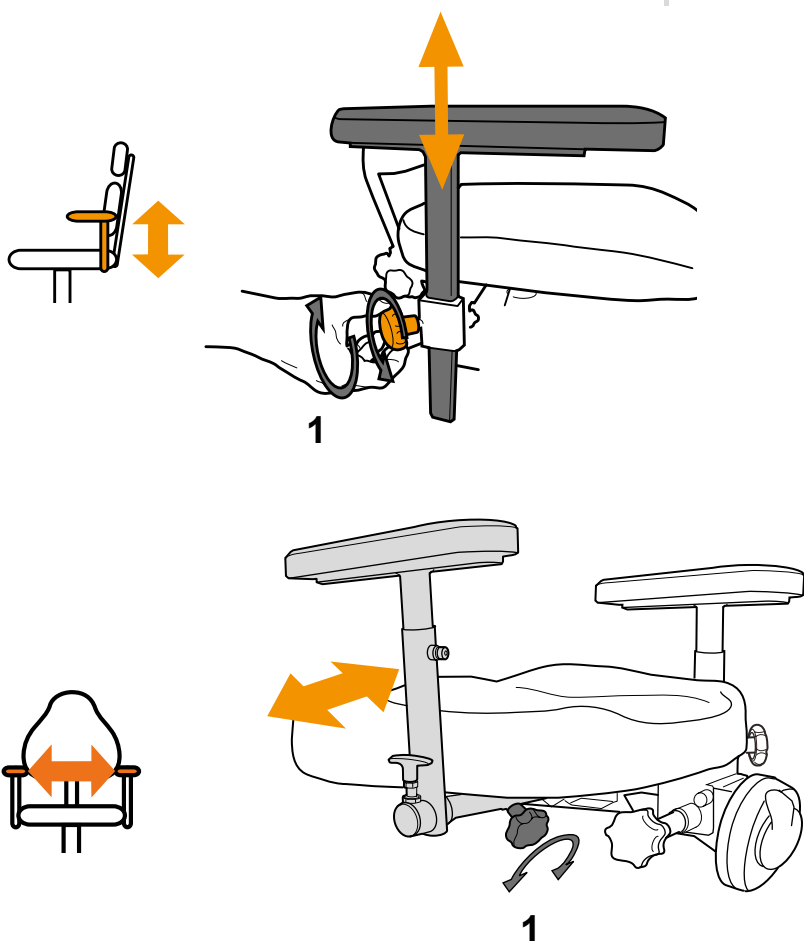
To adjust the width between the arm supports, loosen the wheel nut (1). Adjust to desired width and tighten the wheel. Repeat on the other arm support. And tighten the wheel nut. To adjust the height of the arm support, loosen the wheel lever (2). Adjust to desired height and tighten the wheel lever.

OPTION!

ARM REST - fold back

If the chair is equipped with folded armrest then it is possible to fold the arm support back wards to make sideways transfer easier or to get closer to a table. Push lever in the direction the arrow shows to fold the arm support back wards, use your hand to lift it up where its locked automatically.

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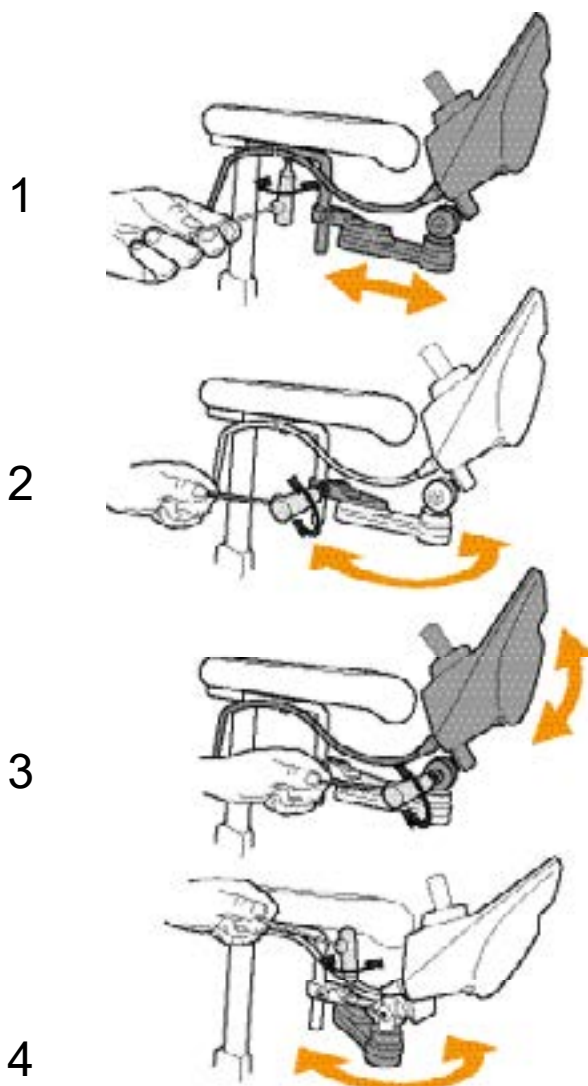
english

CONTROLBOX - Adjustments

The controlbox position can be adjusted by loosening the screws 1, 2 and 3 with allen key nr 5. Loose screw 1 to adjust in depth and screw 2 and 3 to adjust angle and height. Adjust to desired position and tighten the screws. The controlbox can also be swung away when you would like to get close to a table i.e.

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OPTION!

NECK REST - adjust

If the chair is equipped with a neck support it can be adjusted by the knob (1) to adjust to desired position. Move the knob back to the original position to lock.

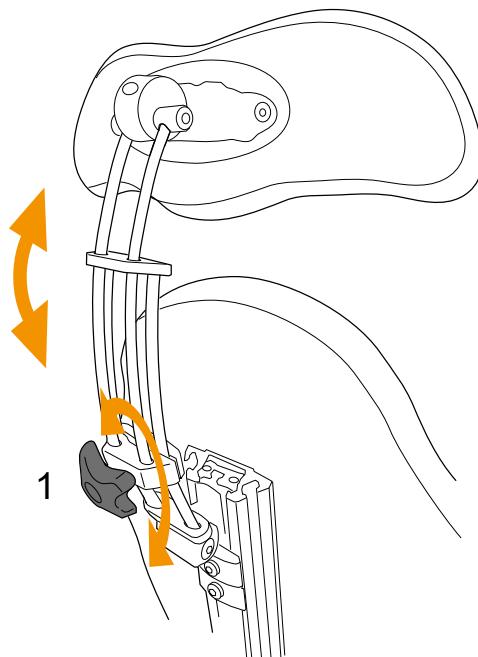
CAUTION PINCH POINT!

When the knob is opened the neck support will come loose and can rapidly fall downwards. Be careful when adjusting the neck support!

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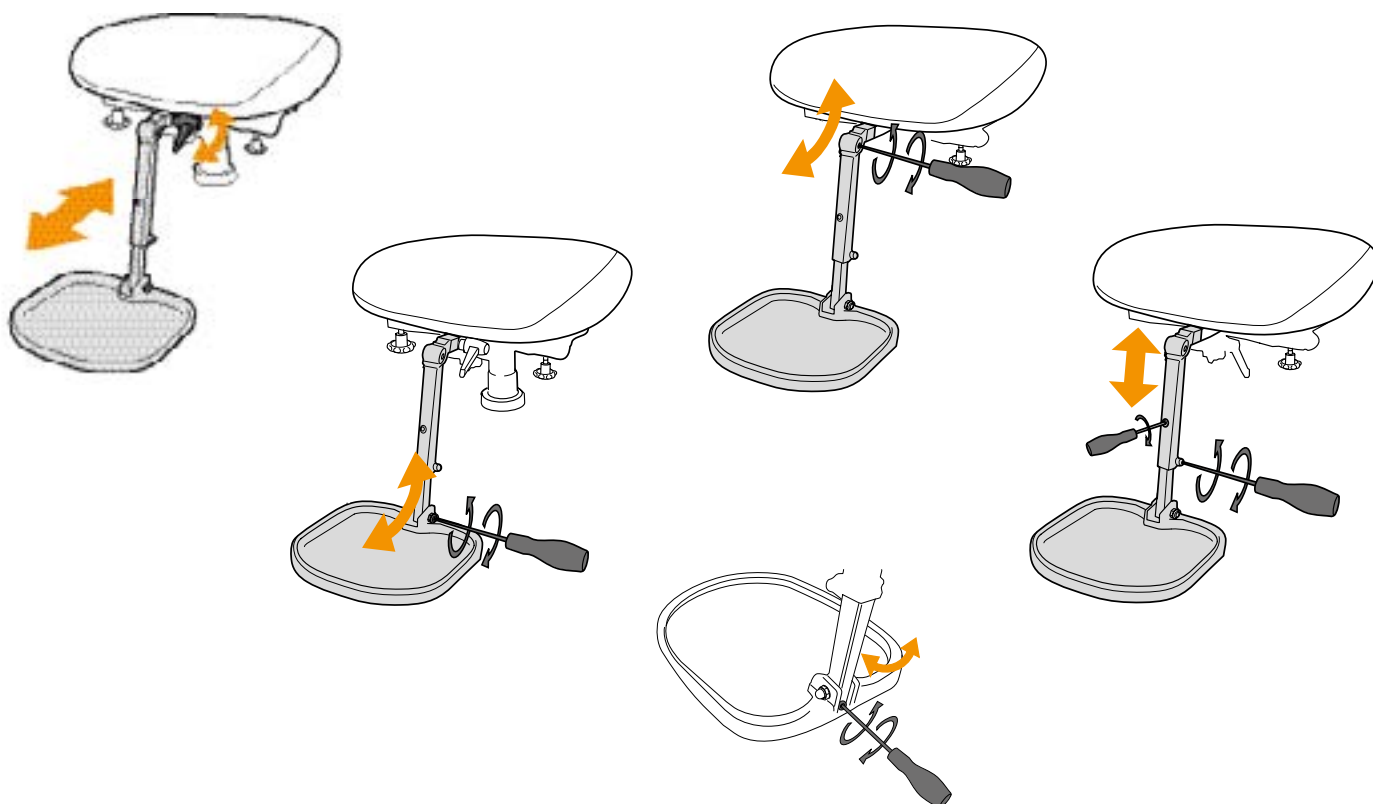
english

LEG REST - single footplate


If the chair is equipped with a single footplate you can adjust the height and lower leg length by loosening screw (1 and 2) with allen key 5 mm. Adjust to desired position and tighten the screws. To adjust depth and the angle of the footplate loosen screws (3 and 4). Adjust to desired position and tighten the screws. The footplate is possible to fold up to make it easier to get in and out of the chair.

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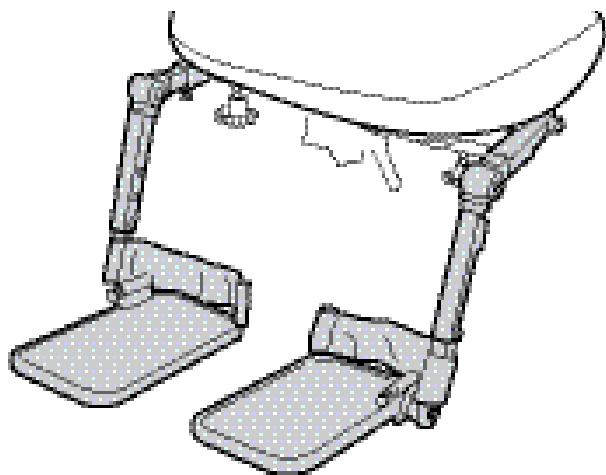
LEG REST - divided footplates

leg rest - divided footplate

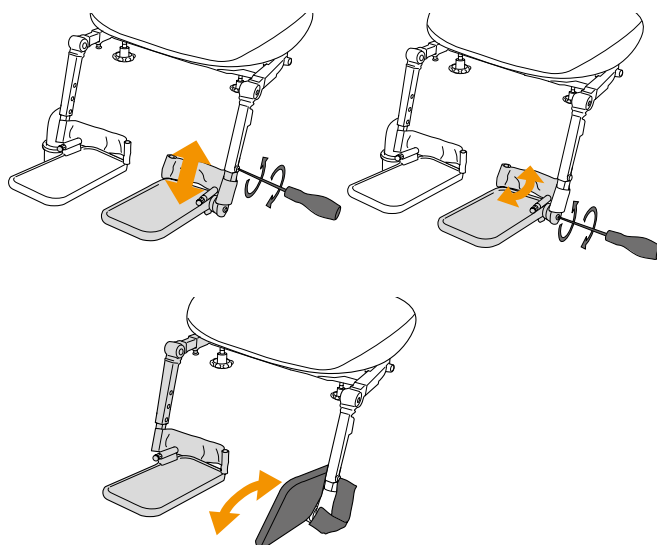If the chair is equipped with divided footplates is's adjusted in height and lower leg angle by loosening screws. Adjust to desired position and tighten the screws. To adjust depth and angle of the footplate loosen screw. Adjust to desired position and tighten the screws. To adjust the legrest position sideways loosen the knob. Adjust to desired position and tighten the knob. The footplate is possible fold up to make it easier to get in and out of the chair.

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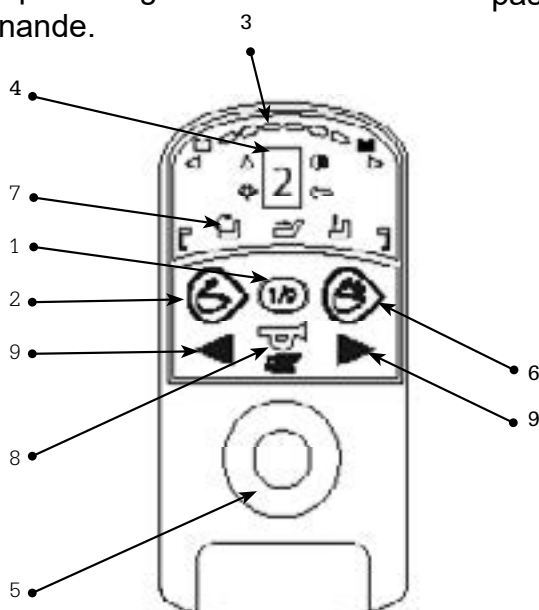
G90A

DRIVING

Push the main switch on and place yourself as comfortable as possible in the chair and let the arm you should use to control the wheelchair rest on the armrest so that your hand can grip the joystick (5) comfortable. Start the electronics by pushing button (1), wait a couple of seconds until the battery indicator (3) has stopped twinkle.

G90T/SHARK

Choose desired drive program (1,2 or 3) with the buttons (2), where 3 is the fastest. The more LEDS the faster the wheelchair goes. Now lean the joystick at the direction you wish to go and the wheelchair goes there. Release the joystick and the wheelchair will stop immediately. This work chair is designed only for indoor use and to climb obstacles of up to 4 cm. These obstacles should be driven over the front wheels diagonally or straight to the obstacle.


While driving:

- Do not push power on/off button.
- Do not start suddenly drive to opposite direction.

**RISK OF INJURE!**

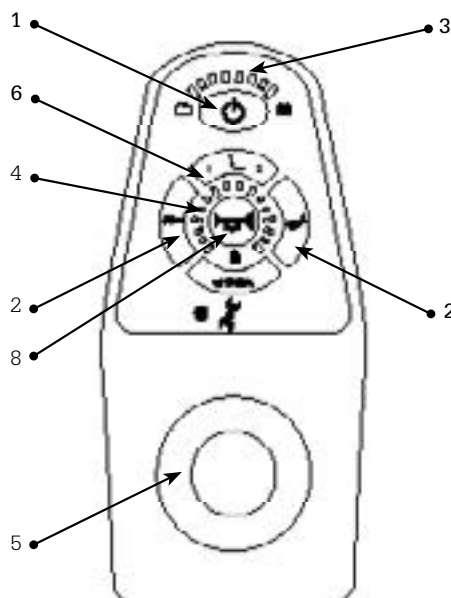NOTE! Make sure that you have a lot of free space around the wheelchair when driving the first time. Practice to drive backwards, turning and forward at different speed to get familiar how the wheelchair behave at different situations. Be careful when passing obstacles like thresholds or similar.

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DRIVE TRAINING

It is important to learn how the wheelchair works in various driving situations. Training is the key to develop your driving skills. But do not train alone.

- **Practice starts.** In the beginning it can be difficult to drive smooth without jolt. Take time to learn to drive smoothly. It is more comfortable and the risk to drive on different things decrease. Try to be soft in the hand / wrist and support your entire forearm against the armrest.

- **Practice braking** so that they become soft. To make sudden braking is not difficult, just release the joystick. Learn how long braking distance your electric wheelchair needs for stopping smoothly.

- **Practice to drive over thresholds.** Low thresholds (5-15mm). Make sure the wheels are pointed straight at the obstacle and pass it with caution. High thresholds (15-30mm), you may have to pass parallel to the obstacle.

- **Practice turns on right and left.** Notice how large floor space is needed to swing the chair a full turn around.

- **Practice to back.** Drive smoothly and slowly. Note that the wheelchair reacts in a completely different way when turning. (Release the joystick immediately), if you lose control and start over again. Lean slightly forward when reversing over a threshold.

- **Practice driving in narrow passages.** Train to drive through the doorway from the side and from the front. Train also backing into tight spaces.

- **Brake.** Move the joystick in the middle position.

- **Emergency stop.** Release the joystick. Stop distance is approx. 1m at 5km/h.

Raised seat lift and seat tilt oblique or back angle changes the center of gravity and increases the risk of tipping! Use seat functions only on flat surfaces and always drive with great caution and at low speed! Keep in mind that a surface that slopes sideways may affect steering of the wheelchair.

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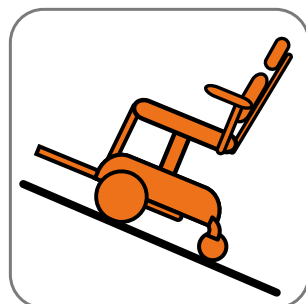
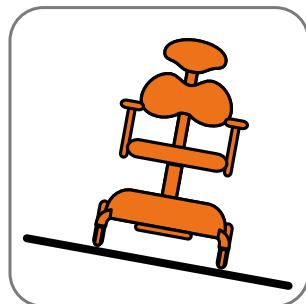
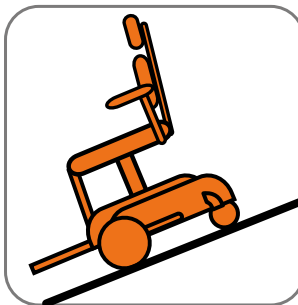
DRIVING INSTRUCTION


The maximum safe downhill slope 6°.



Make sure the seat lift is in its lowest position! Do not turn or brake the wheelchair suddenly on the sloping ground

The maximum safe uphill slope 6°.

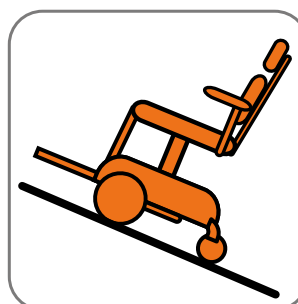The maximum safe lateral slope 6°.

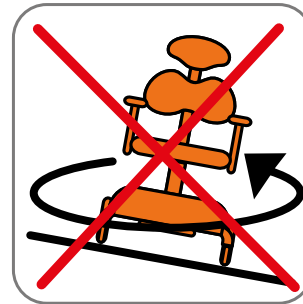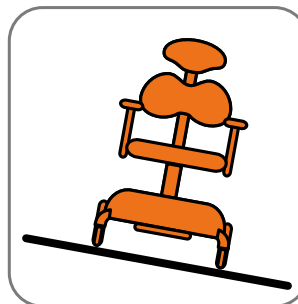The maximum height for a threshold when driving forward 40mm / backward 30mm.

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
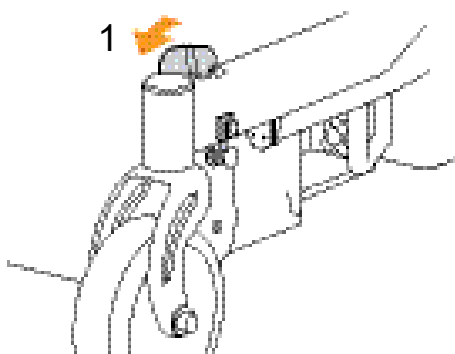
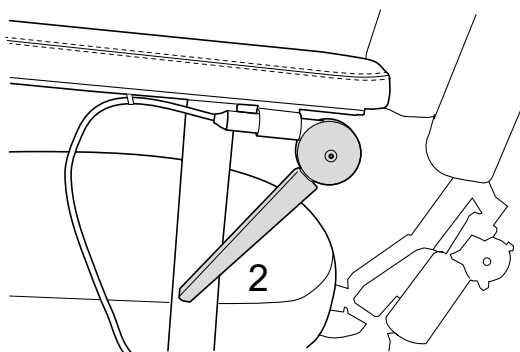
DISENGAGEMENT OF BRAKES

To be able to transport the wheelchair without using the electronics you need to disengage the brakes/motors. This is done by turning of the electronics and pushing the red lever placed on the chassis leg by the rear castor "forward until it's locked in forward position" (1). Now you can transport the wheelchair without motor brakes. To get back into driving you "push the red lever out of the slot" and starts the electronics. If you start the electronics when the brakes are disengaged it will cause an error code on the control box. Turn off the electronics engage the brakes and restart the electronics, the error code will disappear.


RISK for injuries!

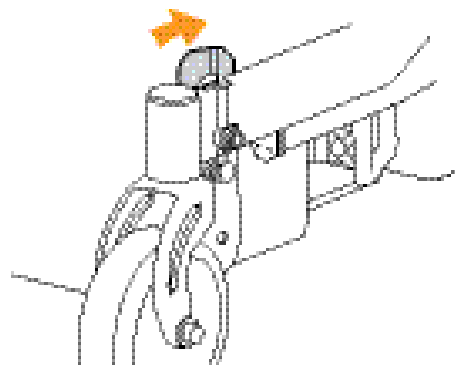NOTE! Make sure that the wheelchair is on flat ground when disengaging the brakes. Otherwise the wheelchair can start rolling away without control.

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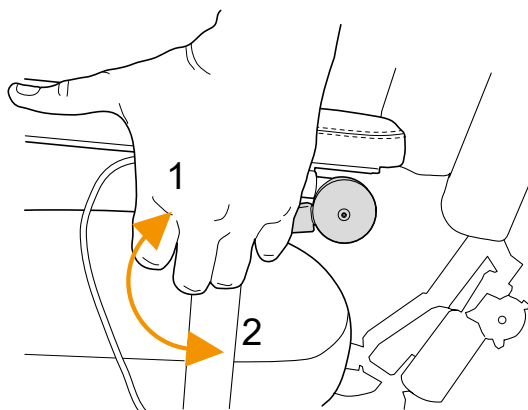english

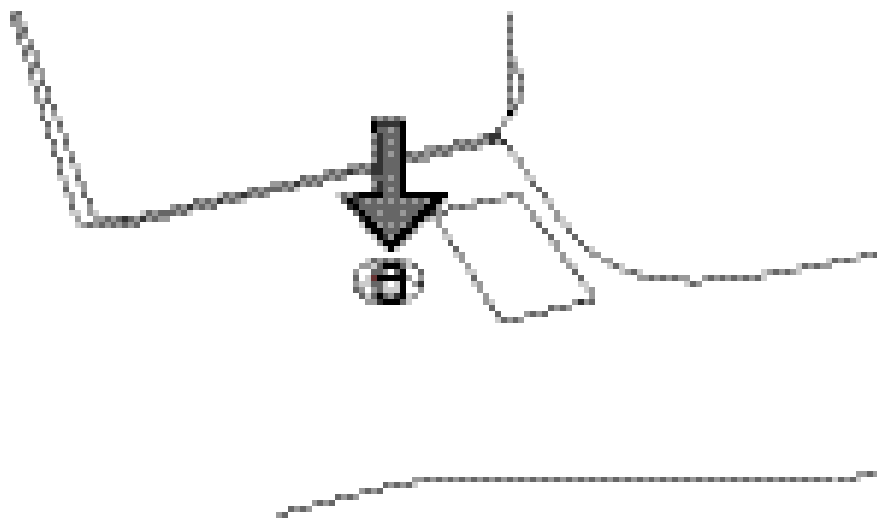
AUTOMATIC FUSE

The automatic fuse is placed on the rear cover on Flexmobil (1). The fuse will disengage if a severe electrical failure occurs on the chair. The button pops out approx. 8 mm and a white line becomes visible. The Automatic fuse can be used as a main-power switch. When the wheelchair is stored for a longer period the fuse should be disengaged.

NOTE!

When the batteries is charged the fuse must be engaged. For transport in a aeroplane the batteries usually must be disconnected and for this you can use the automatic fuse that should be disengaged.

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CHARGING OF BATTERIES

To keep the full power of the batteries as long as possible it is important to charge them regularly. Charge daily, if used daily. Never stop the charging until its completed. Do not charge in confined unventilated

areas due risk of gas formation. When stored the batteries should be charged once a month to prevent them to go under the critical level when the charger cannot start. Do not leave the recharging on for long continuous periods as this is not good for the batteries.

Battery indicator:

The last yellow diode goes down = max 12min remaining driving time(approx. 700m)

One blinking red diode = No driving, batteries must be charged immediately.

NOTE!

If the batteries are damaged or leak, all contact with these should be avoided to prevent corrosive injuries. Contact an authorized workshop as soon as possible! Used batteries should be handed in for recycling. Use only original batteries from Eurovema Mobility AB (Yuasa NPC24-12V 24Ah).

1. Connect the charger plug to the control box on the chair. Located in the front of the control box. See picture.

2. Connect the charger to the wall socket.

3. Check on the charger's indicator lamps that recharging has started. Information on the charger is supplied with the charger.

4. When recharging has been completed, switch off the charger or remove the plug from the wall socket and then the charger plug from the control box.

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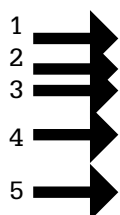
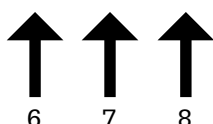
english

WARRANTY

We give a 2 year warranty on our products for fabrication fault/damage. Exceptions are fabric, wheels and batteries where we give 1 year warranty. A normal wear and tear is not accepted as a warranty. We urge our customers to use the products according to the manual

SERVICE

If the wheel chair is used frequently, i.e. every day, it ought to have service once a year. This to control that the function and security of the wheel chair is kept through all its lifetime. If service is needed contact primarily your local dealer.

CE - label

This product has the CE – mark according to the directive 93/42/EEC. The CE mark is on the label. Tested and approved 31. May 2016 according to standard EN12184:2014. Fire tested according to EN 1021 1,2:2014 (Uppholstery: Cigarette and match flame).

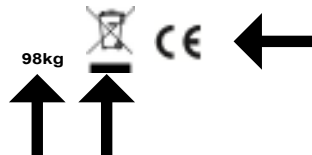
Manufacturing label is located on the back of lifting columns.

1. Manufacture
2. Article + Seatingsystem
3. Article Number
4. Serial Number
5. Class II Equipment
6. Duty Cycle
7. IP Code
8. Input Power Charger
9. Indoor Use
10. Read Instruktions
11. Weight Chair
12. Recycle
13. CE Marking

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SERVICE AND MAINTENANCE

You get more joy out of your power wheelchair if it is served correctly. The batteries must be charged, the seat must be washed and wiped, the tires must be checked regularly and periodically the wheelchair may need have a drop of oil in their joints to not seize up. Keep Service Journal!

CHARGE:

Turn off the main switch before charging. Charge the battery every day, if you use the wheelchair daily and unplug the charger when the batteries are fully charged. Do not charge in wet areas such as shower room or in confined, unventilated areas.

CLEANING:

Wipe the surface of the wheelchair with a damp cloth with a mild detergent. Place the chair on plain surface for drying. The upholstery can be cleaned with any existing brands on the market. Covers can be machine washed in 60 degrees Celsius.

Never use the high-pressure washer for cleaning.

FUNCTION:

If loose screws or loose parts is discovered somewhere on the wheelchair or a change in drive characteristic, immediate repair should take place when this could affect safety. Contact your local agent or the manufacturer for help and questions. Be sure to tighten all screws and knobs after seat adjustments.

STORAGE:

If the wheelchair should be unused for a longer period (a month or more) it should be stored in a clean, dampfree room tempered space. The batteries should be charged once a month to prevent them to go under the critical level when the charger cannot start.

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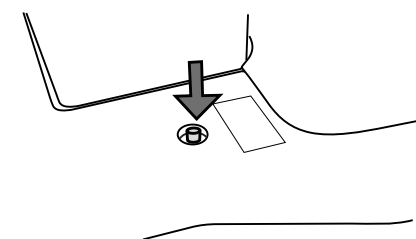
BATTERY REPLACEMENT

1. Turn off the main switch.
2. Pull and lift the cover.
3. Loosen both straps.
4. Disconnect all cable connectors.
5. Lift the protective cover.
6. Follow the wiring diagram is pasted on the inside of the battery cover!
Always start unscrewing the minus (-) cable from the negative terminal. When reassembling always start with the red marked positive cable (+). Cables for the batteries should not be mixed with each other. They are marked +1, +2, -1 and -2.
7. Now unscrew the positive cable (+) the plus (+) terminal. When reassembling always start with the positive cable (+).
8. Remove the old battery and replace the new one.


Assemble in reverse order except the plus and the negative lead.

Always observe the following safety procedures


- Use protective glasses and gloves.
- Never short circuit the battery. Carry neither never necklaces, bracelets or rings of metal that may come in contact with battery terminals and may cross-circuit the battery.

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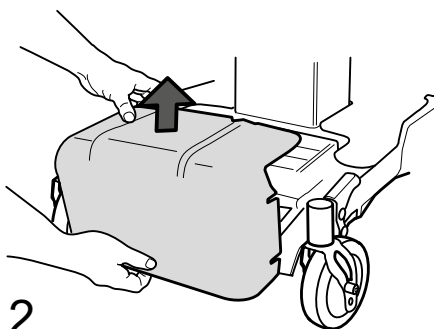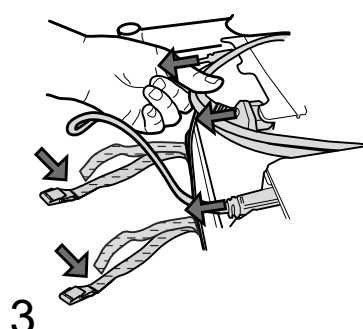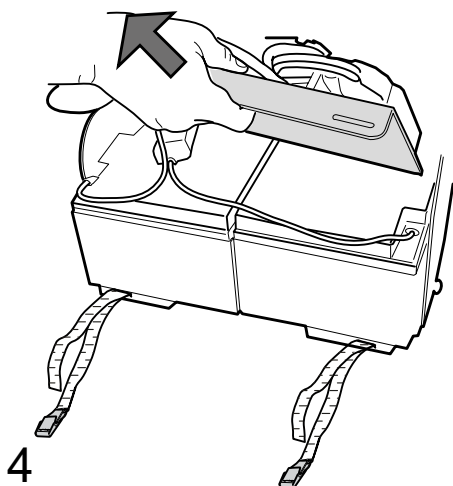
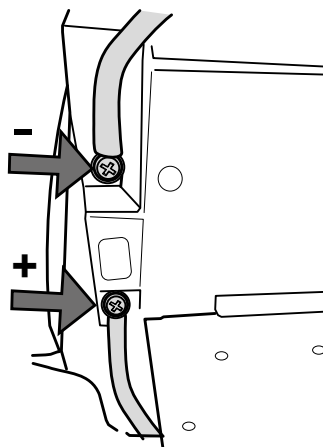
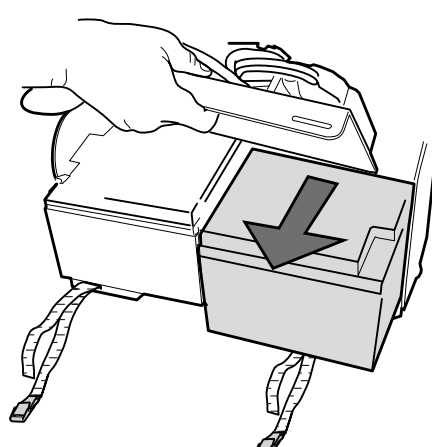
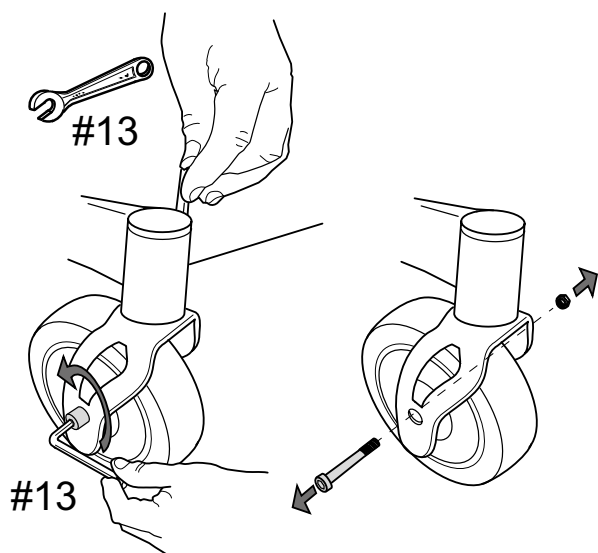
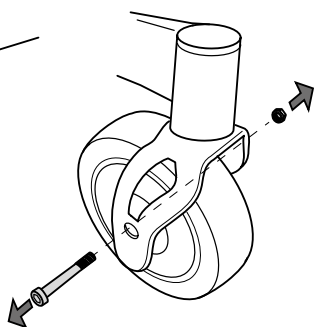
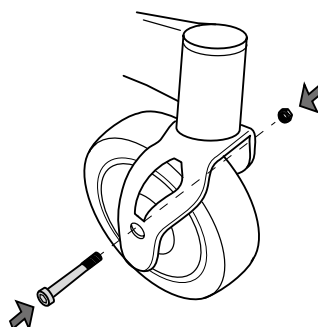
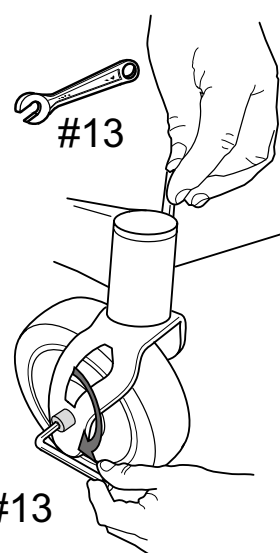
english

CHANGE OF CASTOR

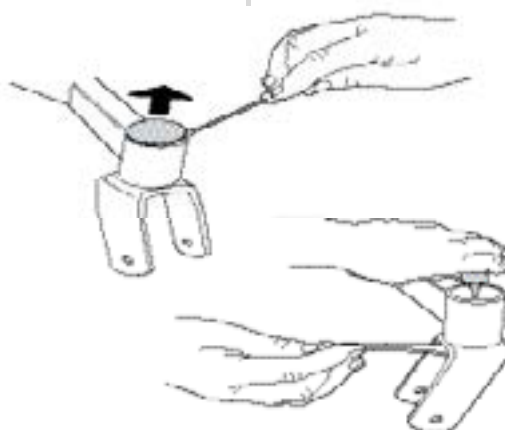
1. Place a wooden block under the wheel arm.
2. Loosen the wheel bolt with 13mm key and 5mm allen key.
3. Remove the bolt and lock nut.
4. Replace the castor wheel and mount the nut and bolt.
5. Tighten the bolt and lock nut.

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MOUNTING AND DEMOUNTING FORK

Remove the cap. Unscrew the 5mm allen screw with the allen key and 17mm spanner. Mount the new fork and fasten the screw.

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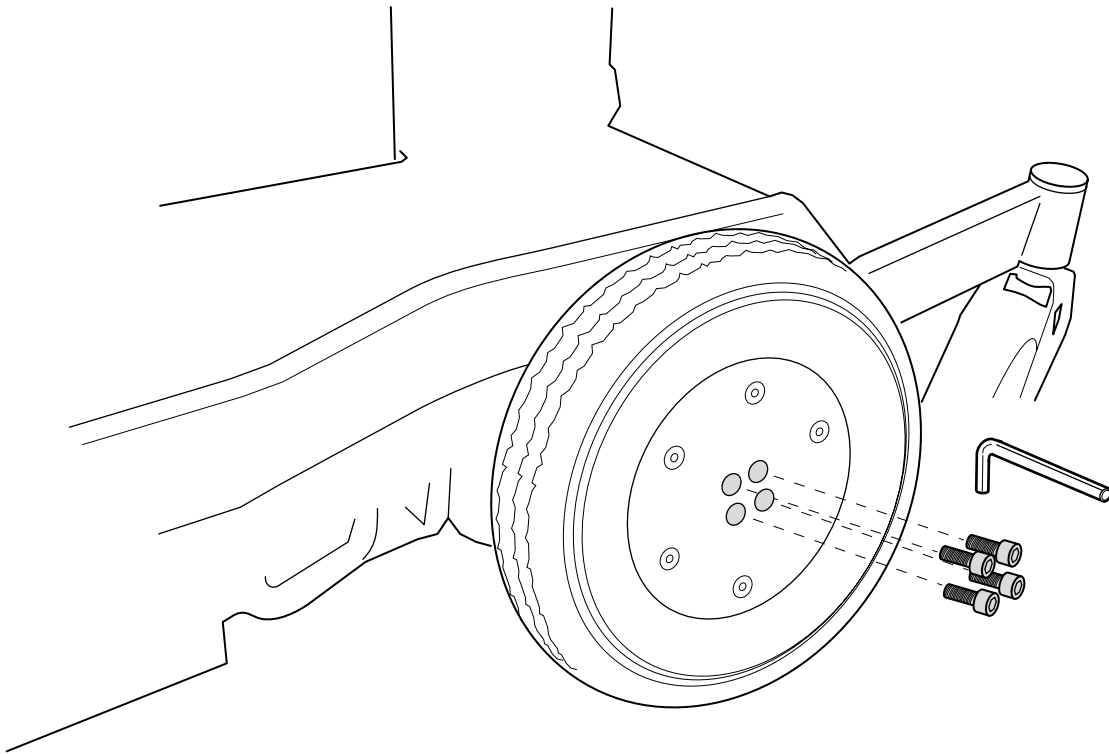
english

REPLACING OF THE DRIVE WHEEL

1. Place a wooden block under the slightly raised chassis and ensure that it is stable.
- 2 Heat up the bolts little by using heat air pistol and loosen the bolts with a allen head 3mm key.
3. Remove the wheels and fit a new wheel. Apply some threadglue (eg. Loctite 243) in the threads before tightening the bolts.

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Action!

The wheelchair doesn't start?

The automatic fuse is released?
 The batteries is discharged?
 Loose cables?

Check the fuse - see the manual.
 Charge the batteries!
 Contact service!

The batteries can not be charged?

The automatic fuse is released?
 The batteries is totally discharged?
 The charger is broken?

Check the fuse - see the manual
 Contact service!
 Contact service!

Battery indicator drops quickly despite charged batteries?

The batterier starts getting bad?
 Error in the electronics?

Contact service!
 Contact service!

The status LED is blinking on the controlbox?

Electronics has been started when brakes are disengaged?
 An electric fault has occurred on the wheelchair?

Turn off and engage the brakes- see the manual!
 Contact service!

The wheelchair is running slow?

Is the right drivprogram selcted?

Select a quicker driveprogram- see the manual!

The wheelchair don't manage to climb obstacles?

A too slow driveprogram is selected?

Select a quicker driveprogram- see the manual!

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TRANSPORT

When transporting Miniflex or Flexmobil in a motor vehicle it is important that the brakes are engaged. See section "Disengagement of brakes". The chair should be secured with tie down straps.

Special brackets to be mounted on the chair are available as accessories.

The wheelchair is not designed for transport with the user seated in the chair.

You can reduce the transport size by removing the armrest, backrest and the legrest.


The batteries in the wheelchair are maintenance free and of the type GEL/AMG and approved for transport in aircrafts.

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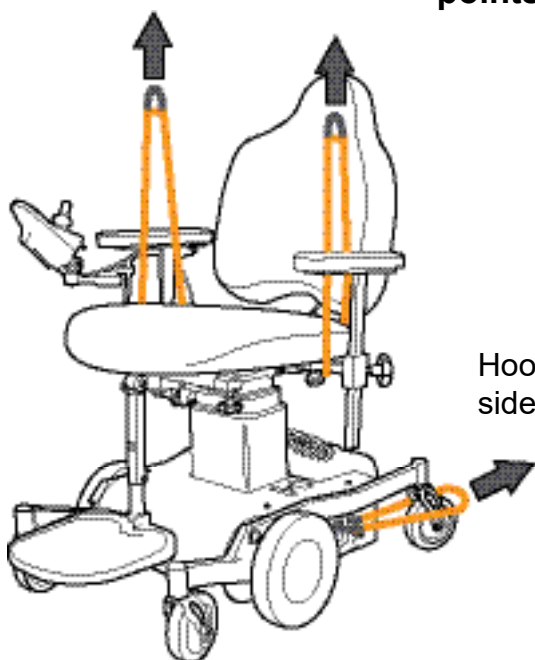Instructions for transfer of the the wheelchair user

- Prepare where the user will be transferring to.
- Ask if the user can help and tell him or her what you will do even during the transfer.
- When transferring to a wheelchair, be sure to have armrest removed or flipped backwards to avoid contact or injury.
- Remove the footrest, or swing away to prevent feet from becoming tangled when performing the wheelchair transfer.
- Position the wheelchair either at the same height or slightly lower to assist in a safe and proper wheelchair transfer.
- Depending on transfer method, having the wheelchair parallel to transfer point may also allow for a more safe and easy wheelchair transfer.
- Ensure the brakes are activated and the wheelchair is stable.

Perform the one person wheelchair transfer

- Ensure the user is utilizing a gait or transfer belt.
- Stand as close to the user as possible. Be sure footing is as stable as possible.
- Proper shoes are also recommended as sandals or flats do not provide support for a safe wheelchair transfer.
- Assist the user in nearing the edge of the wheelchair.
- Be sure the user's feet are under his or her body.

Picture 1. Lift on the belt while the user is leaning slightly forward and rises her/him up.

Picture 2. Keep the user's weaker knee in between your own to assist and to slowly sit onto wheelchair.

- Securely position the patient on the chair and replace the armrests.

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Batteri

Recycling instructions

We hope you have noticed, that the new Flexmobile i6 is made of fewer parts and with plenty of metal, instead of plastics. Metal is easy to recycle and helps us keep our environment clean. This was our goal when we started to design this new Eco-friendly wheelchair.

If you want to help us to achieve this goal, you must ensure that it ends up in recycling.

The proper disposal of the product

The product and its materials must be dismantled, sorted and handed in for recycling according to national or local regulations and practices.

Sorting guidelines

1. Metals

Chassis, wheel forks, seat cross, lifting column, armrests bracket, joystick bracket, legrests bracket, neckrest bracket, backrest bracket, electronic plate, freewheel pedal, shafts, bearing, wire, spacer, washers, bolts, screws, nuts.

2. Combustible

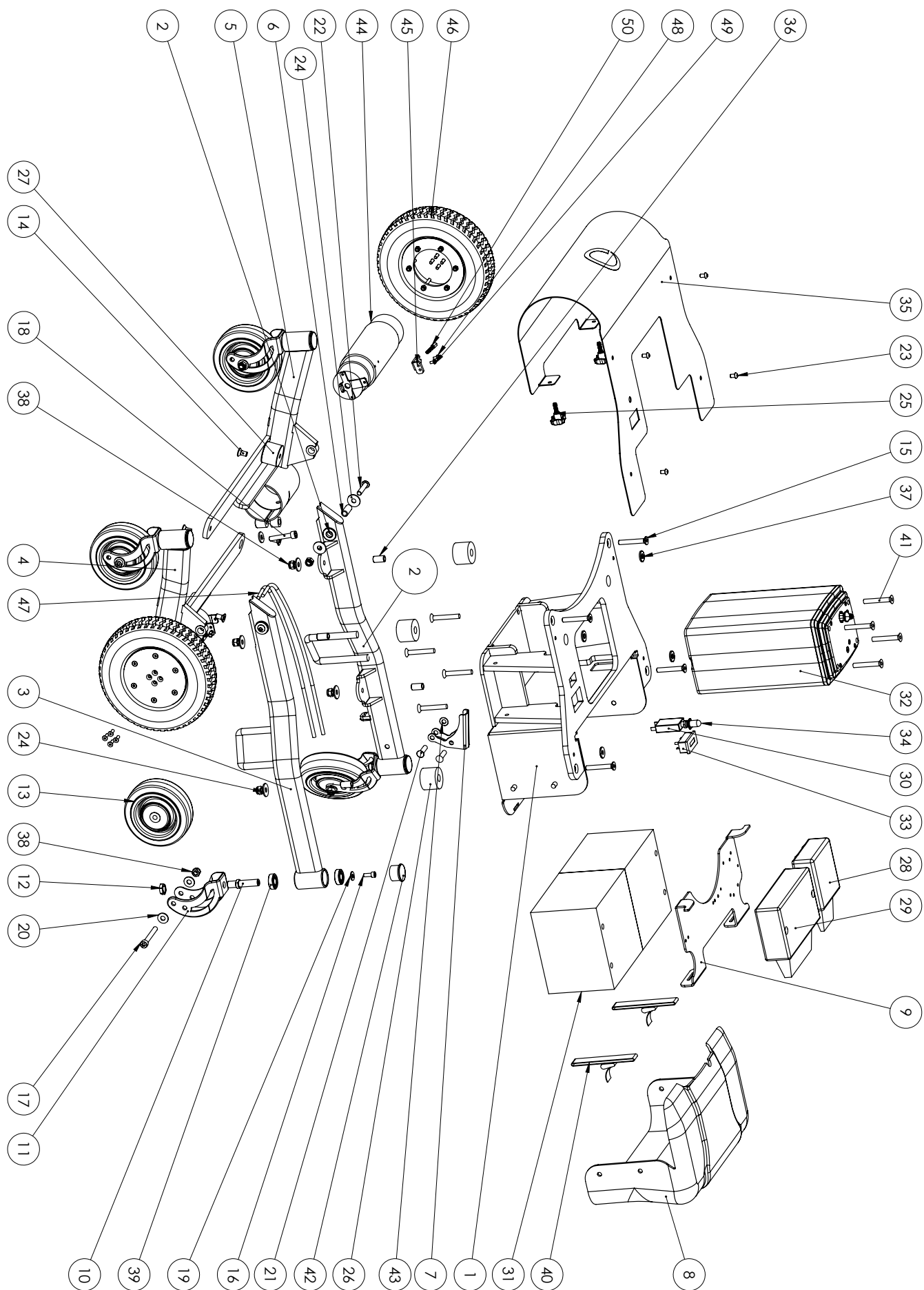
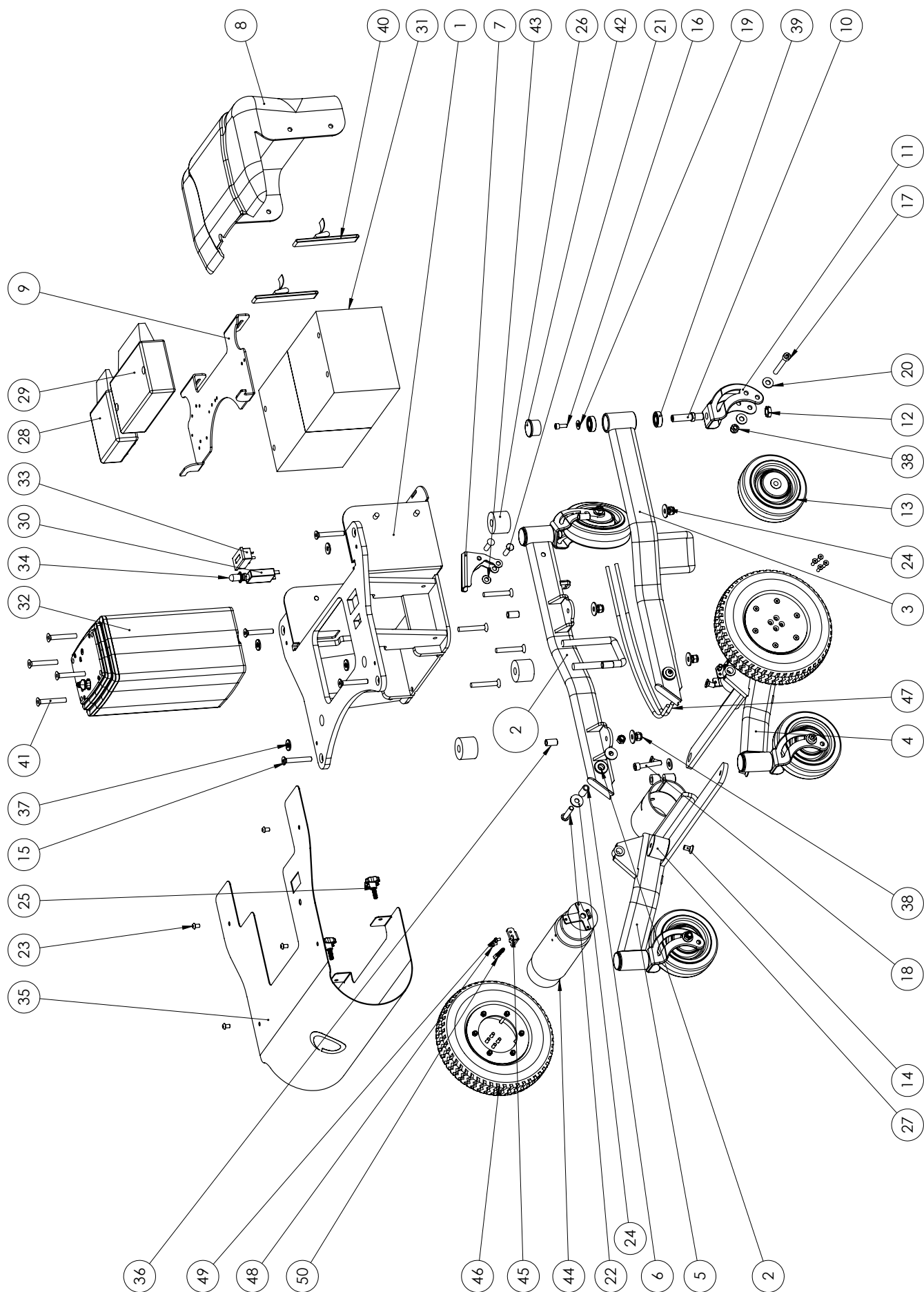
All fabric covers, foam parts, rubber wheels, knobs, dampers, battery cover, cable lock, straps, seals.

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