

Spare Parts Catalogue

Version: 2020/07/08a EN



Invacare®
Comet^{PRO} / Comet^{ALPINE+} / Comet^{ULTRA}



Yes, you can.®

General Information

For any order of spare parts the following information is required:

- Product type*
- Product name & model*
- Serial Number*
- Part-Number*
- Part-Description*

Exploded view drawings and code numbers are not contractual and subject to modification, in relation to Invacare® continuous product improvement policy to meet end users needs.

The reproduction or use of this work, or of any substantial part of this work, in any form or by any electronic, mechanical or other means, including photocopying, scanning and recording, and in any information storage and retrieval system, including electronic transmission, is forbidden without the prior written permission of Invacare® International Sarl.



Yes, you can.®

TABLE OF CONTENTS

Information & User Manuals

User Manuals Comet Pro / Alpine+ / Ultra	5
Label HMI	6

Chassis, Shrouds & Lights

Front Frame	7
Front Shroud	8
Rear Frame / Motor Swing	9
Rear Shrouds	10
Colour Panels (6 pieces)	11
Light System	12

Seating

Seats & Headrests	13
Seat Post	14
Seat Bracket	15
Slide Assembly	16
Armrests	17

Wheels

Wheels - 5" rim	18
Wheels - 8" rim	19
Anti-tipper wheel	20

Propulsion

Motors & Differentials	21
------------------------------	----

Brakes

Brake assembly	22
----------------------	----

Steering

Steering	23
Suspension	24
Steering linkage	25

Batteries, Cables & Charger

Battery & Cables	26
Chargers	27

Electronic

Electronic components	28
Powerbox & Limit trigger speed limit	29

Options & Accessories

Safety	30
Warning Triangle	31



Yes, you can.®

TABLE OF CONTENTS

Seating	32
Walking support	33
Rollator bracket	34
Storage	35
Storage box rear bracket	36
Bumpers	37
Other Accessories	38

User Manuals Comet Pro / Alpine+ / Ultra

BG	Български	HE	עברית	PL	Polski
CS	Čeština	HR	Hrvatski	PT	Português
DA	Dansk	HU	Magyar	RU	Русский
DE	Deutsch	IT	Italiano	RO	Română
EN	English	JA	日本語	SA	العربية
ES	Español	LV	Latviešu	SK	Slovenčina
ET	Eesti	LT	Lietuvių	SL	Slovenščina
FI	Suomi	NL	Nederlands	SV	Svenska
FR	Français	NO	Norsk	UK	Українська

languages.iso

Pos.	Part number	Description	Valid from → until	Qty
EN	SP1603455	User Manual Comet Pro / Alpine+ / Ultra - English		1
DE	SP1608099	User Manual Comet Pro / Alpine+ / Ultra - German		1
IT	SP1608100	User Manual Comet Pro / Alpine+ / Ultra - Italian		1
NL	SP1608101	User Manual Comet Pro / Alpine+ / Ultra - Dutch		1
NO	SP1608102	User Manual Comet Pro / Alpine+ / Ultra - Norwegian		1
HR	SP1638425	User Manual Comet Pro / Alpine+ / Ultra - Croatian		1
CS	SP1638456	User Manual Comet Pro / Alpine+ / Ultra - Czech		1
DA	SP1638457	User Manual Comet Pro / Alpine+ / Ultra - Danish		1
ET	SP1638458	User Manual Comet Pro / Alpine+ / Ultra - Estonian		1
FI	SP1638459	User Manual Comet Pro / Alpine+ / Ultra - Finnish		1
FR	SP1638460	User Manual Comet Pro / Alpine+ / Ultra - French		1
HU	SP1638461	User Manual Comet Pro / Alpine+ / Ultra - Hungarian		1
LV	SP1638469	User Manual Comet Pro / Alpine+ / Ultra - Latvian		1
LT	SP1638505	User Manual Comet Pro / Alpine+ / Ultra - Lithuanian		1
PL	SP1638506	User Manual Comet Pro / Alpine+ / Ultra - Polish		1
PT	SP1638550	User Manual Comet Pro / Alpine+ / Ultra - Portuguese		1
SK	SP1638551	User Manual Comet Pro / Alpine+ / Ultra - Slovenian		1
ES	SP1638552	User Manual Comet Pro / Alpine+ / Ultra - Spanish		1
SV	SP1638553	User Manual Comet Pro / Alpine+ / Ultra - Swedish		1
UK	SP1638554	User Manual Comet Pro / Alpine+ / Ultra - Ukrainian		1

Label HMI




1650303.iso

Pos.	Part number	Description	Valid from → until	Qty
1	SP1649113	HMI Label Comet PRO (Sticker Sheet 27 pcs)		1
2	SP1649114	HMI Label Comet ULTRA (Sticker Sheet 27 pcs)		1
3	SP1649118	HMI Label Comet ALPINE+ (Sticker Sheet 27 pcs)		1

Front Frame


1637038-000

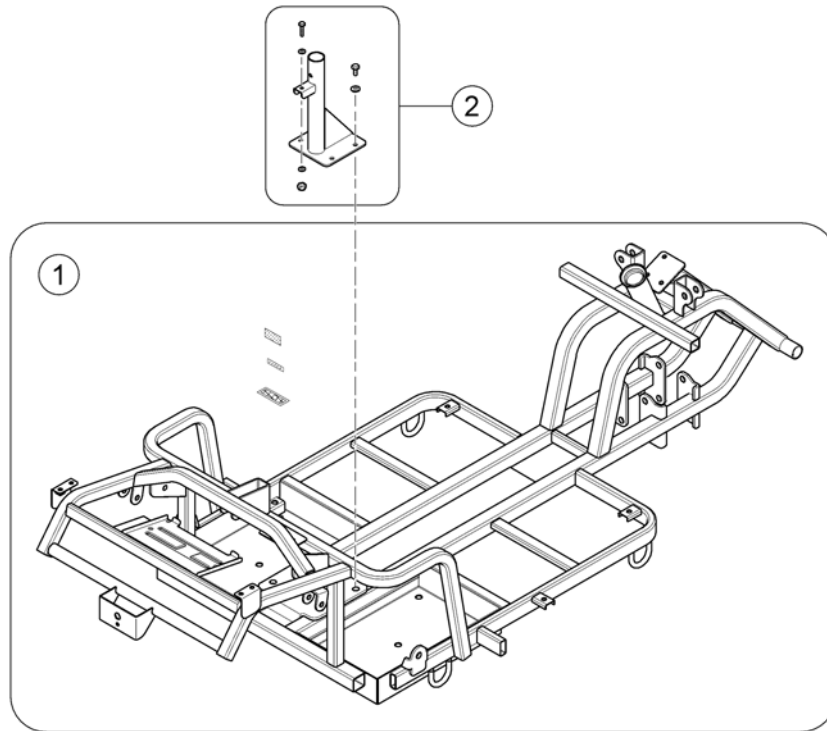

Pos.	Part number	Description	Valid from → until	Qty
1	SP1636815	Front frame		1
2	SP1636817	Lower seat bracket		1


Front Shroud

1637108-00

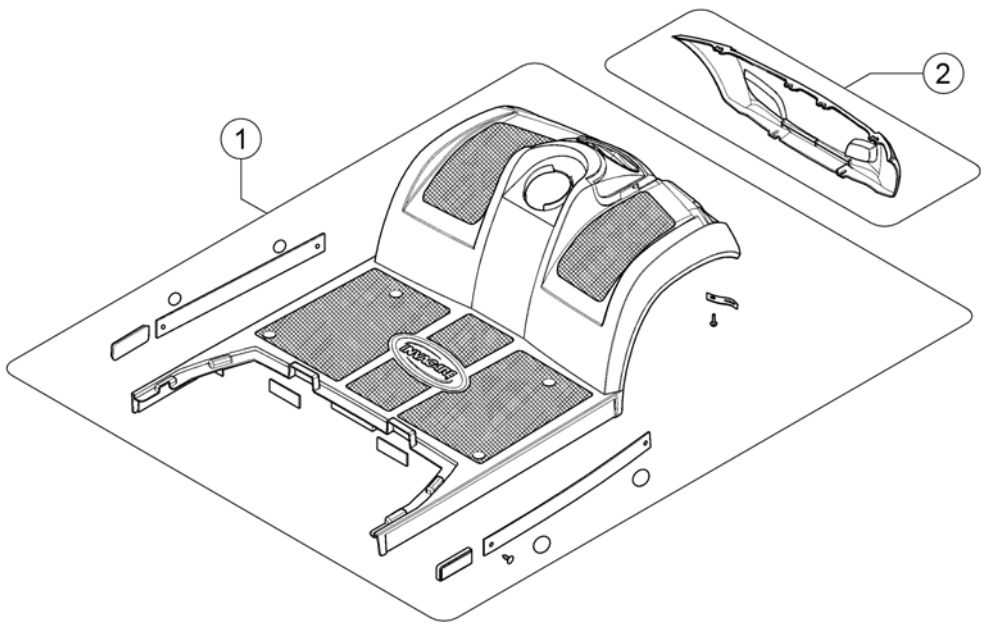

Pos.	Part number	Description	Valid from → until	Qty
1	SP1636818	Front Shroud, cpl.		1
2	SP1608232	Cover, front bumper		1

Rear Frame / Motor Swing

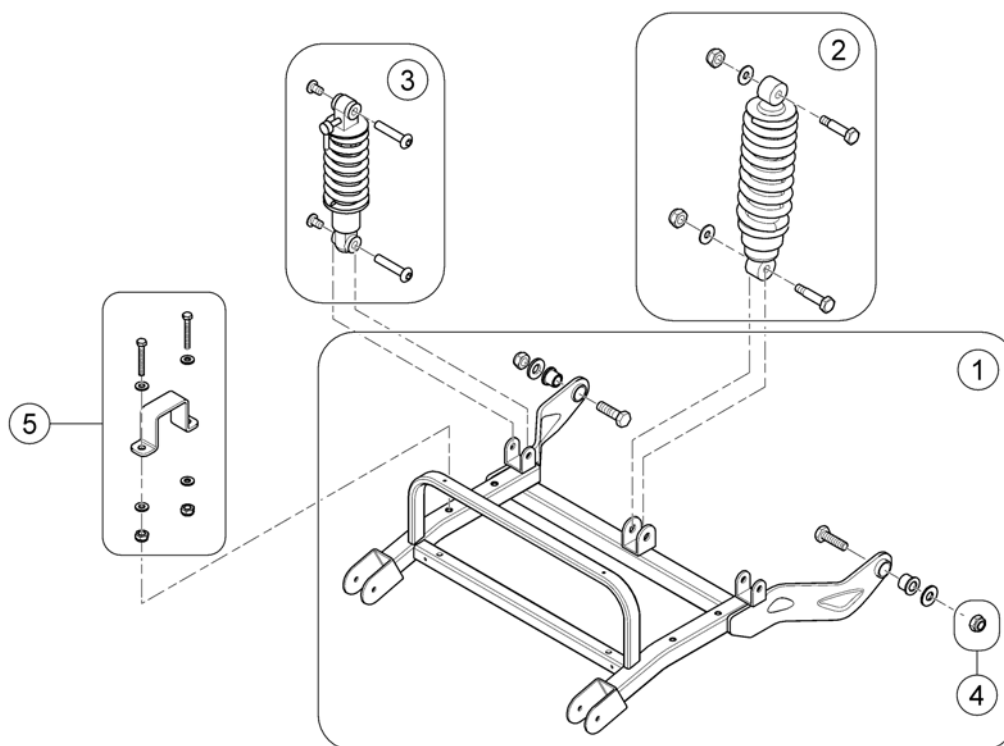


1637105.050


Pos.	Part number	Description	Valid from → until	Qty
1	SP1636825	Rear swing arm cpl. <i>Design change - fully retrofittable</i>		1
2	SP1645145	Shock absorber (central), cpl. <i>Comet PRO / ALPINE+</i>		1
3	SP1636838	Shock absorber (sideways), cpl. <i>Comet ULTRA</i>		2
4	SP1505589	Nylon Nut		2
5	SP1636837	Holder, Transaxle, cpl.		2

Rear Shrouds

1637107/06

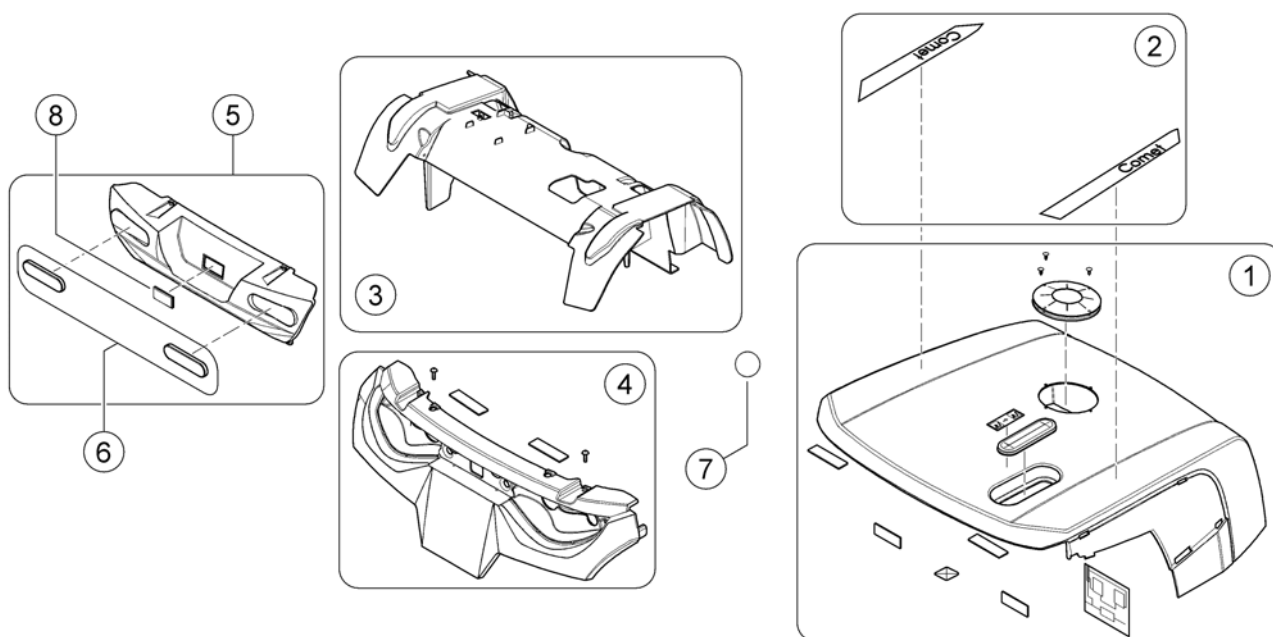

Pos.	Part number	Description	Valid from → until	Qty
1	SP1608258	Rear shroud (w/o model labels)		1
2	SP1608259	Label Set - Pro		1
2	SP1637306	Label Set - Alpine+		1
2	SP1637462	Label Set - Ultra		1
3	SP1608261	Rear Fender		1
4	SP1636839	Cover, rear bumper (upper part)		1
5	SP1608260	Cover, rear bumper (lower part)		1
6	SP1518036	Reflector (with E-Mark)		1
7	SP1608262	Indication stickers, shock absorber (Ultra only)		2
8	SP1521879	Plug, Bumper Rear		1

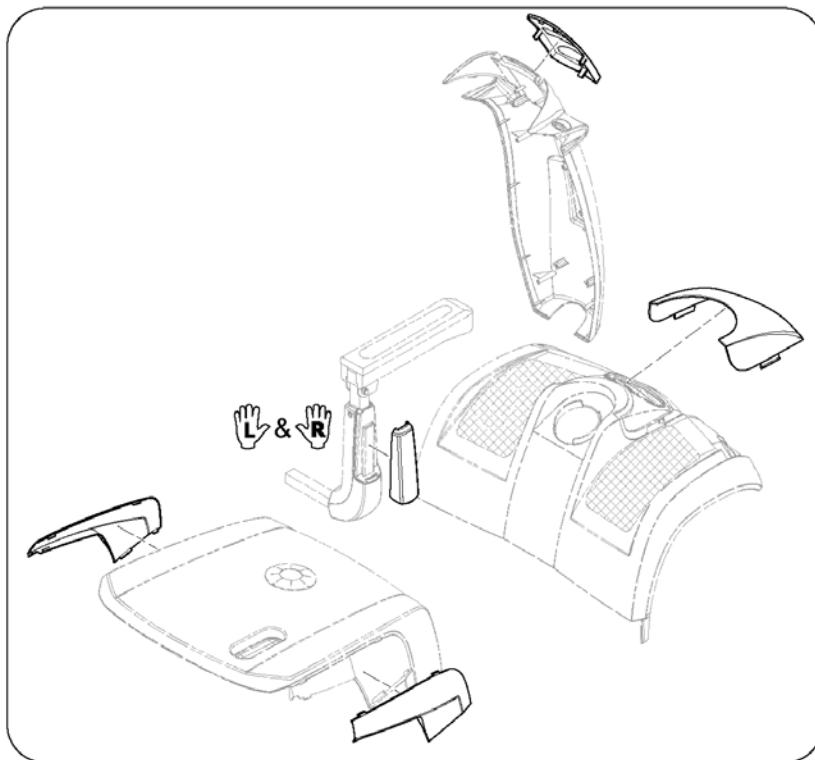
Colour Panels (6 pieces)

1637123 iso

Pos.	Part number	Description	Valid from → until	Qty
1	SP1604096	Color panels - Alium Blue <i>Replaces SP1608823</i>		1
2	SP1608821	Color panels - Marble White (Classic Line)		1
3	SP1608822	Color panels - Jade Green (Classic Line)		1
4	-	Color panels - Sapphire Blue no longer available - please choose Alium Blue. <i>Replaced by SP1604096</i>		1
5	SP1608831	Color panels - Diamond Silver (Classic Line)		1
6	-	Color panels - Jasper Red no longer available - please choose Garnet Red. <i>Replaced by SP1654684</i>		1
7	SP1654684	Color panels - Garnet Red (Classic Line)		1
8	SP1637415	Color panels - Luxor Beige (Premium Line)		1
9	SP1637416	Color panels - Crystal Copper (Premium Line)		1
10	SP1637417	Color panels - Carmine Red (Premium Line)		1
11	SP1637418	Color panels - Electric Blue (Premium Line)		1
12	SP1637419	Color panels - Ionic Chrome (Premium Line)		1
13	SP1637420	Color panels - Polygon, shades of red		1
14	SP1637421	Color panels - Polygon, shades of blue		1
15	SP1637422	Color panels - Hexagon, shades of white		1


Light System

1540472/90

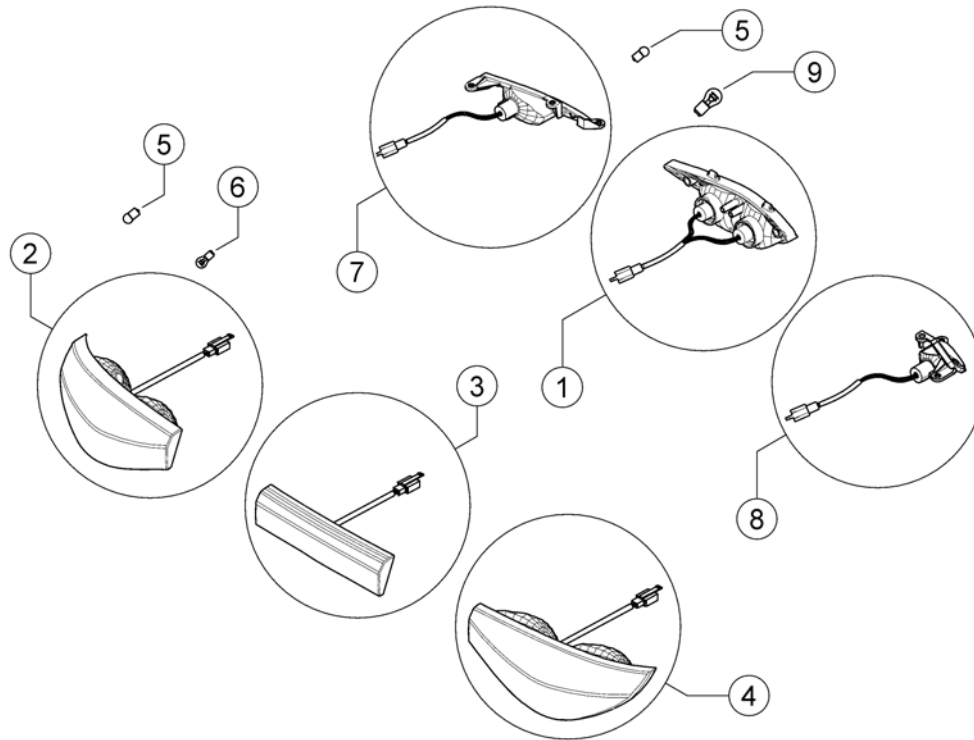

Pos.	Part number	Description	Valid from → until	Qty
1	SP1527473	Head light, cpl.		1
2	SP1518034	Rear Light incl. Turn Signal - LH		1
3	SP1518035	Brake Light, cpl.		1
4	SP1518033	Rear Light incl. Turn Signal - RH		1
5	SP1522030	Bulb Turn Signal 24V/10W		4
6	SP1522034	Bulb Rear Light 24V/5W		1
7	SP1517733	Turn signal 4-Wheel, front - LH		1
8	SP1517734	Turn signal 4-Wheel, front - RH		1
9	SP1527474	Bulb Head Light 12V/15W		2


Seats & Headrests

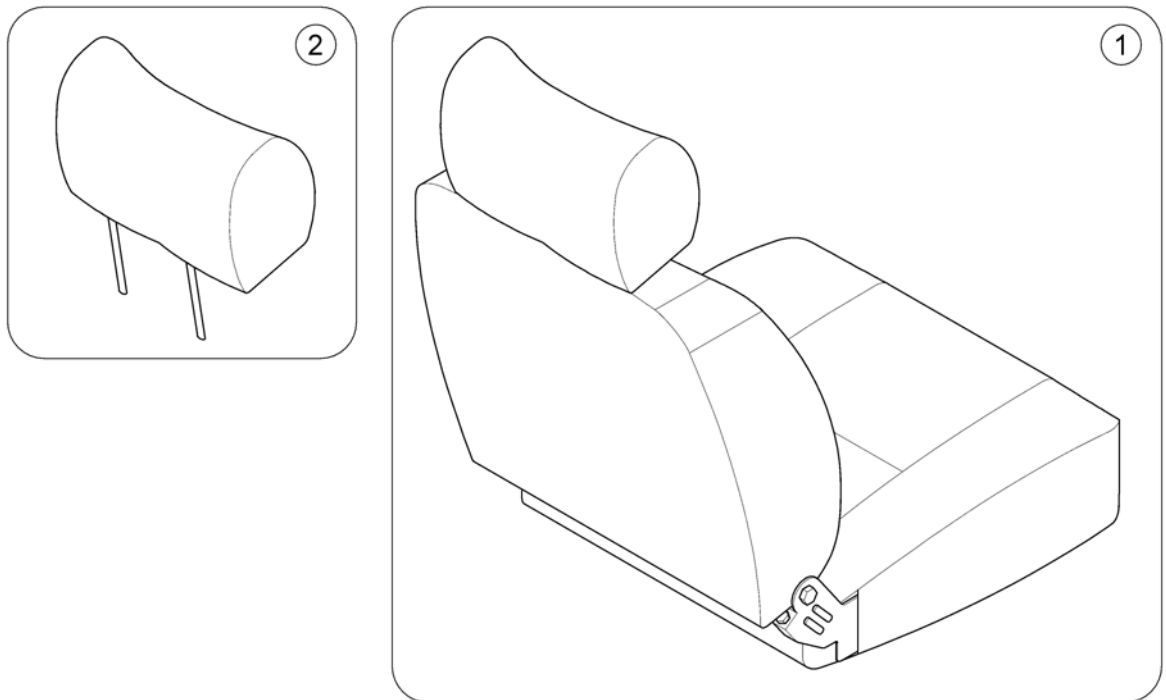

Pos.	Part number	Description	Valid from → until	Qty
1	SP1608540	Seat 20" Comfort		1
1	SP1586116	Seat 20", double white stitching <i>Deluxe Seat (DE = Komfort Sitz)</i>		1
1	SP1608556	Seat 20" Premium		1
1	SP1652116	Seat 20" Deluxe Low (SH 450mm) <i>Not ULTRA!</i>		1
1	SP1584887	Seat 20" HD <i>NORWAY only!</i>		1
1	SP1586115	Seat 26" Ultra (HD), double white stitching		1
2	SP1529380	Headrest Standard/Comfort Seat		1
2	SP1642145	Headrest Premium Seat		1
2	SP1602195	Headrest HD Seat		1


Seat Post

1637084.00

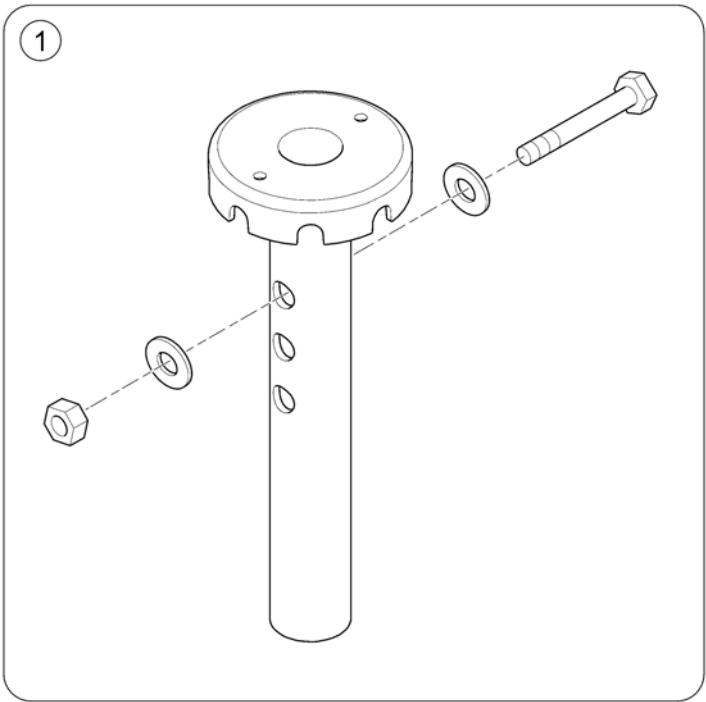

Pos.	Part number	Description	Valid from → until	Qty
1	SP1521691	Seat Post		1
1	SP1640520	Seat Post - Comet ULTRA		1

Seat Bracket

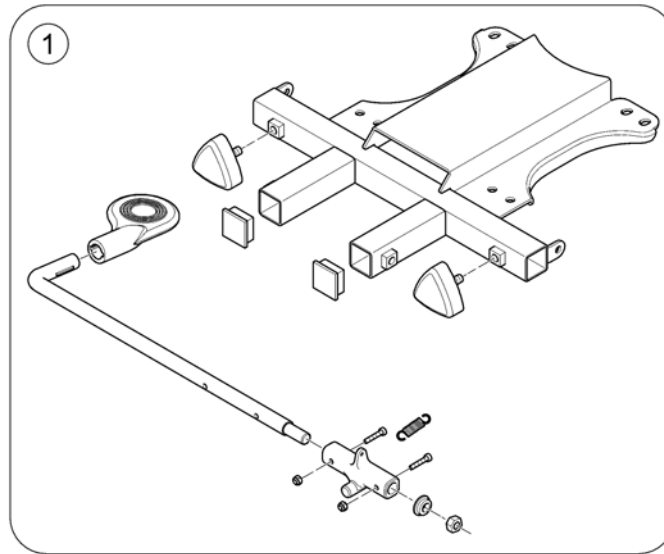


1637085.iso

Pos.	Part number	Description	Valid from → until	Qty
1	SP1636855	Seat Bracket Assembly, cpl.		1
1	SP1636856	Seat Bracket Assembly HD, cpl. <i>Comet ALPINE+ / ULTRA only!</i>		1




Slide Assembly

1637086-00

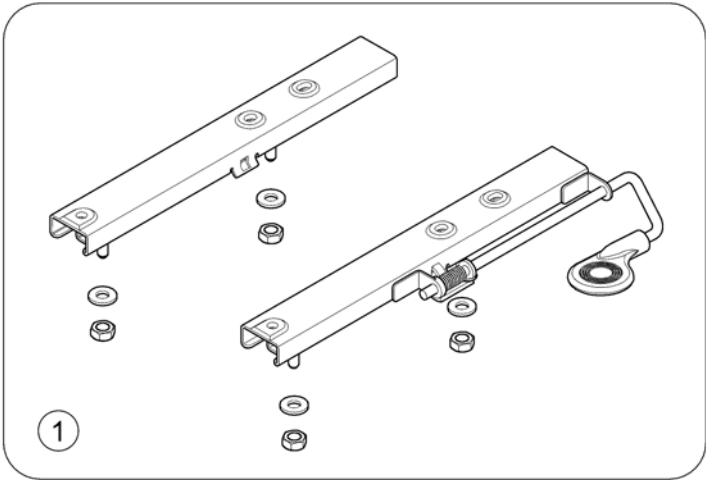


Pos.	Part number	Description	Valid from → until	Qty
1	SP1521690	Slide Assembly		1
1	SP1645013	Slide Assembly - OrionPRO 3W (user weight > 127kg)		1

Armrests

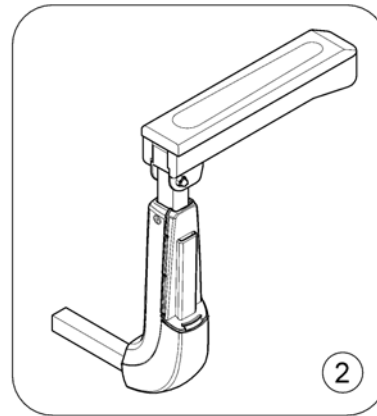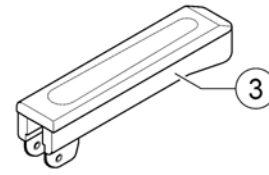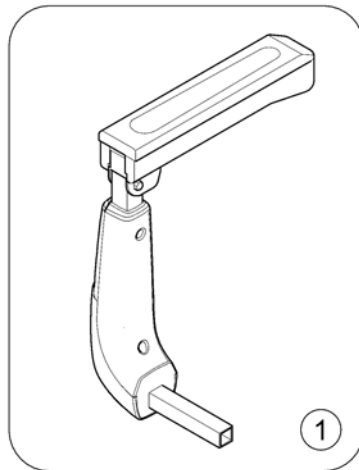

1637396.iso

Pos.	Part number	Description	Valid from → until	Qty
1	SP1636858	Armrest, cpl - LH		1
1	SP1641799	Armrest HD, cpl. - LH		1
2	SP1636857	Armrest, cpl. - RH		1
2	SP1641800	Armrest HD, cpl. - RH		1
3	1509683	Rest, Handrail		2

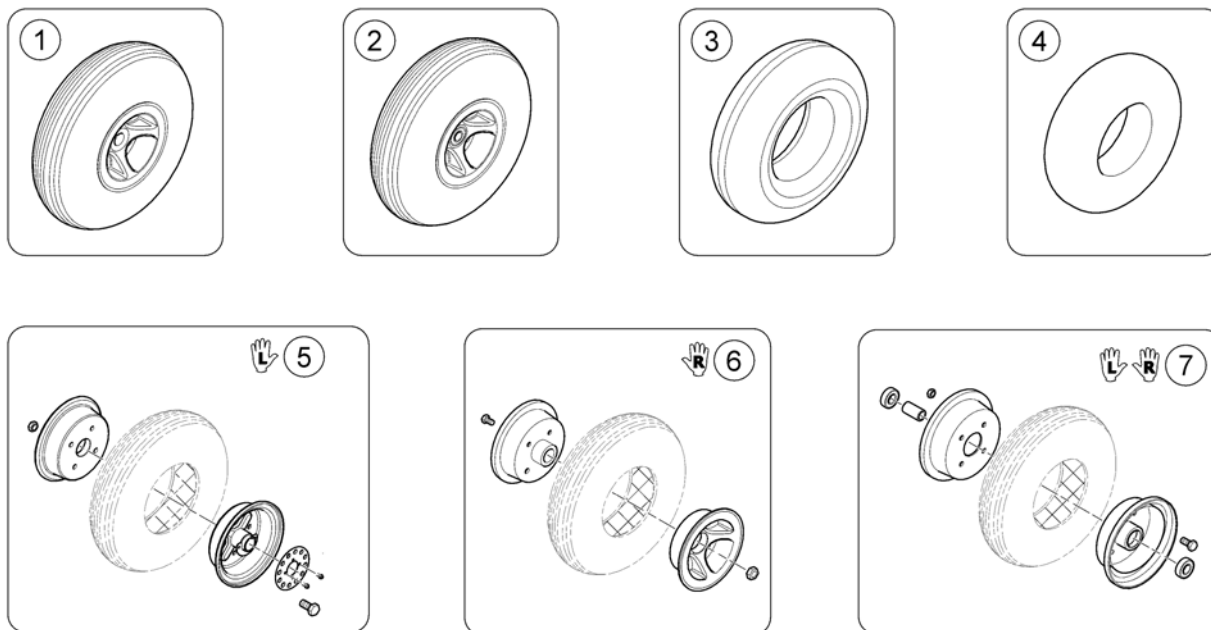
Wheels - 5" rim

1639752.iso

Pos.	Part number	Description	Valid from → until	Qty
1	SP1637429	Rear Wheel 13", pneumatic, 5" rim, RH <i>Design change! Replacement: SP1654383</i>	→ 2019-10-09	1
1	SP1654383	Rear Wheel 13", pneumatic, 5" rim, LH/RH (new design) <i>New design! Replaces SP1608676</i>	2019-10-10 →	1
1	SP1608775	Rear Wheel 13", puncture proof, 5" rim, LH <i>Design change! Replacement: SP1654385</i>	→ 2019-10-09	1
1	SP1654385	Rear Wheel 13", puncture proof, 5" rim, LH (new design) <i>New design! Replaces SP1608775</i>	2019-10-10 →	1
1	SP1654386	Rear Wheel 13", puncture proof, 5" rim, RH (new design) <i>New design! Replaces SP1637431</i>	2019-10-10 →	1
2	SP1608594	Front Wheel 13", pneumatic, 5" rim (old design)		1
2	SP1654351	Front Wheel 13", pneumatic, 5" rim (new design) <i>New Design! Replaces: SP1608594</i>		1
2	SP1654357	Front Wheel 13", puncture Proof, 5" rim (new design) <i>New design! Replaces SP1608696</i>	2019-10-10 →	1
3	SP1640471	Tyre 13" (4.00-5) - black		1
3	SP1586119	Solid Tyre PU 4.00-5 - black		1
4	SP1527714	Tube 4.00-5		1
5	SP1638864	5" Rim Kit, Rear (old design) <i>LH - Design change! Replacement: SP1654350</i>	→ 2019-10-09	1
5	SP1654350	5" Rim Kit, Rear (new design) <i>LH & RH - New design! Replacing: SP1638864 & SP1638838</i>	2019-10-10 →	1
6	SP1654350	5" Rim Kit, Rear (new design) <i>LH & RH - New design! Replacing: SP1638864 & SP1638838</i>	2019-10-10 →	1
7	SP1638837	5" Rim Kit, Front <i>Design change! Replacement: SP1654355</i>	→ 2019-10-09	1
7	SP1654355	5" Rim Kit, Front (new design) <i>New design!</i>	2019-10-10 →	1

Wheels - 8" rim


1639753.00

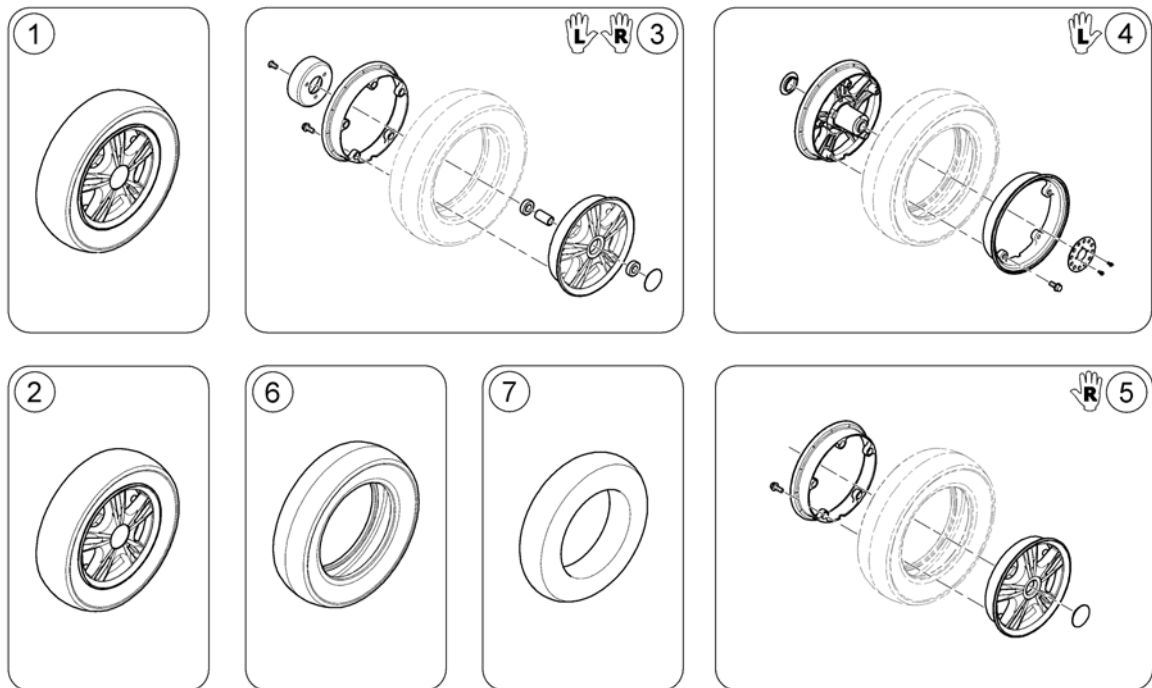

Pos.	Part number	Description	Valid from → until	Qty
1	SP1608797	Front Wheel 13", pneumatic, Low Profile, 8" rim		1
2	SP1608819	Rear Wheel 13", pneumatic, Low Profile, 8" rim, LH		1
2	SP1637433	Rear Wheel 13", pneumatic, Low Profile, 8" rim, RH		1
3	SP1638855	8" Rim Kit, Front		1
4	SP1638842	8" Rim Kit, Rear LH		1
5	SP1638856	8" Rim Kit, Rear RH		1
6	SP1638843	Tyre 13" for 8" rim		1
7	SP1638979	Tube 13" for 8" rim		1


Anti-tipper wheel

1637088-00

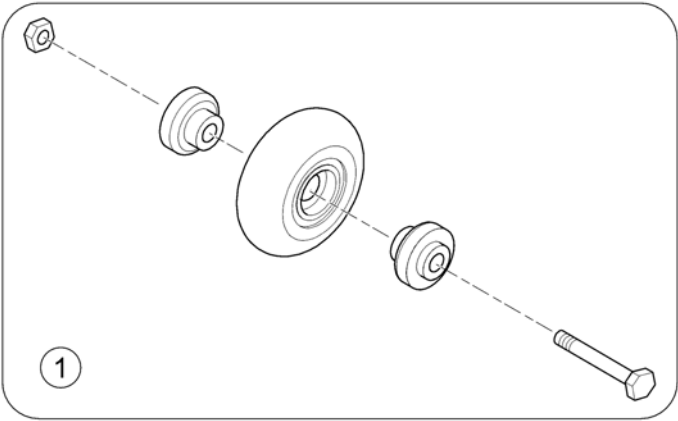

Pos.	Part number	Description	Valid from → until	Qty
1	SP1522021	Wheel, Anti-Tipper		1

Motors & Differentials

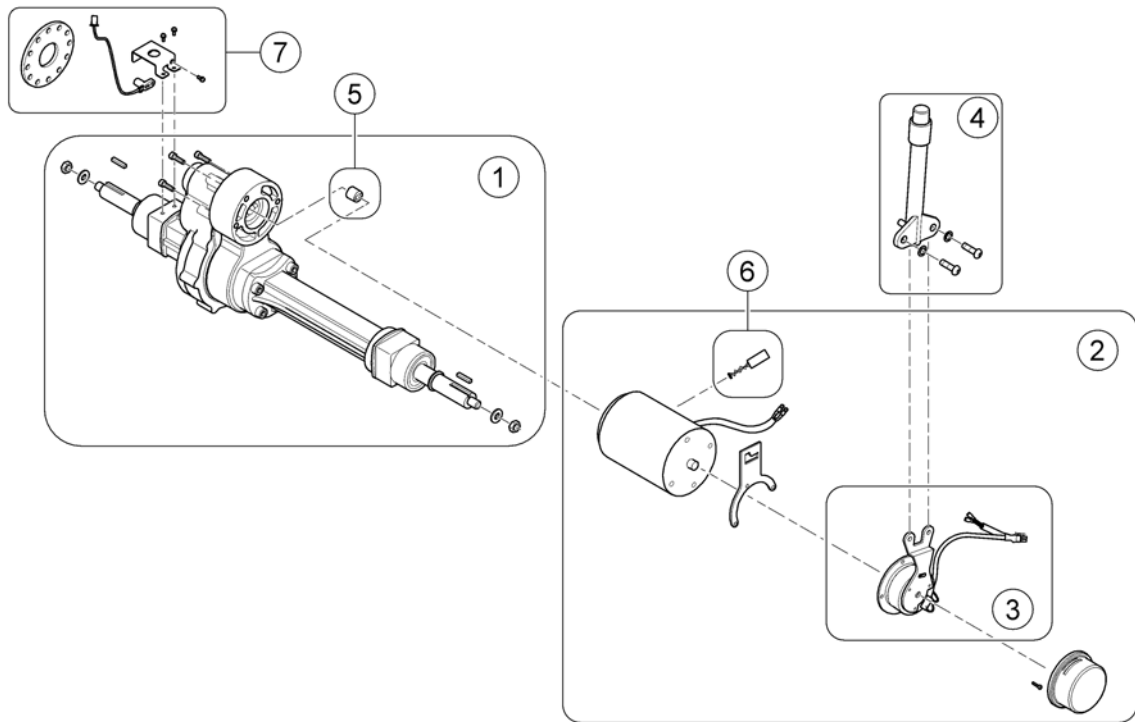


1637109.iso

Pos.	Part number	Description	Valid from → until	Qty
1	SP1636893	Differential - Pro		1
1	SP1636894	Differential - Alpine+ & Ultra		1
2	SP1608264	Motor 10 km/h - Pro		1
2	SP1608265	Motor 12.8 km/h - Pro		1
2	SP1608266	Motor 15 km/h - Pro		1
2	SP1636875	Motor 10 km/h - Alpine+ & Ultra		1
3	SP1636879	Clutch & Brake 10 km/h (Pro)		1
3	SP1636891	Clutch & Brake 12.8 & 15 km/h (Pro)		1
3	SP1636892	Clutch & Brake 10 km/h (Alpine+ / Ultra)		1
4	SP1505672	Disclutching Lever L=113,2mm, cpl.		1
5	SP1571584	Guide		1
6	SP1636876	Carbon brush 10 km/h (Pro)		1
6	SP1636877	Carbon brush 12.8 & 15 km/h (Pro)		1
6	SP1636878	Carbon brush 10 km/h (Alpine+ / Ultra)		1
7	SP1643083	Kit, Speed Sensor, cpl. replaces item SP1636874		1

Brake assembly


1638622-00

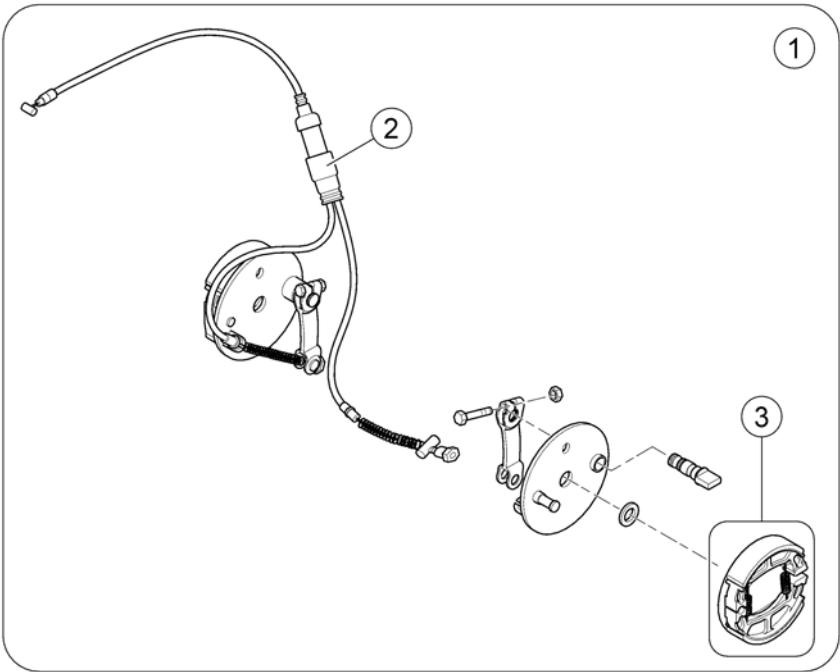

Pos.	Part number	Description	Valid from → until	Qty
1	SP1636873	Brake Assembly		1
2	SP1608231	Brake Cable		1
3	SP1518418	Brake Shoes, cpl. (95mm)		1


Steering

1637057/98

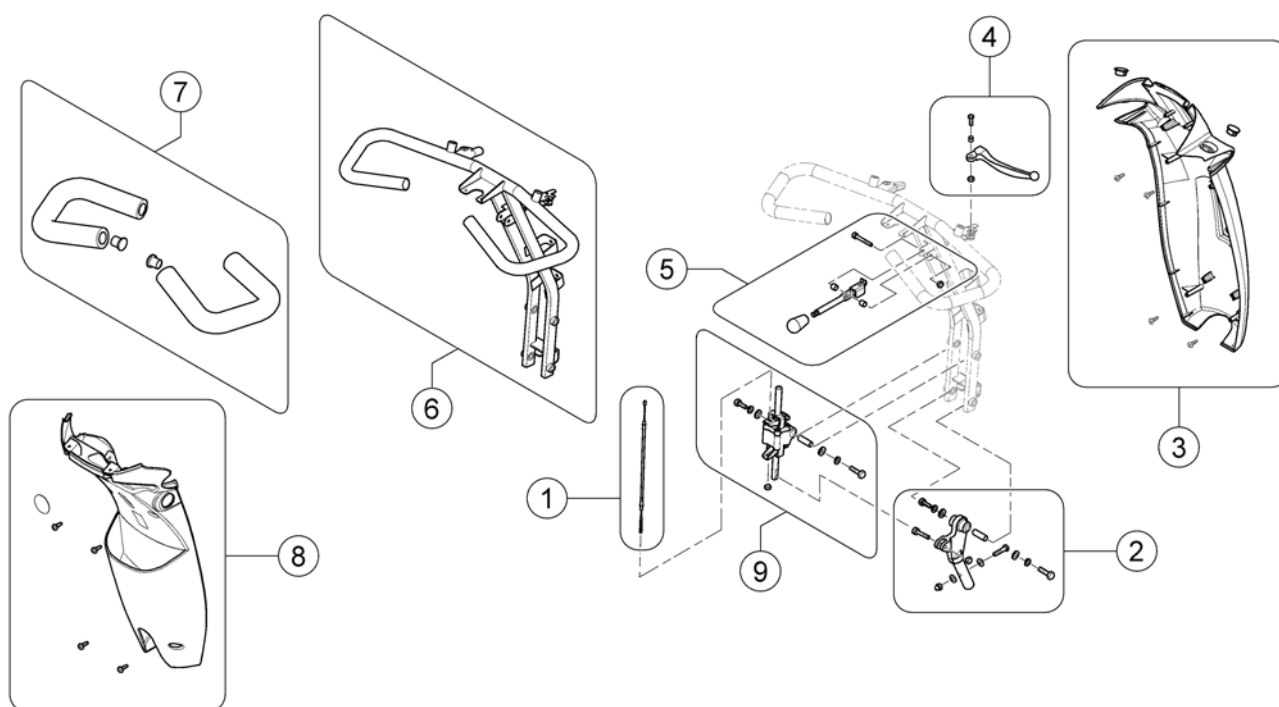


Pos.	Part number	Description	Valid from → until	Qty
	SP1637540	Reflective Sticker		1
1	SP1608201	Cable, steering adj.		1
2	SP1636812	Joint, steering adjustment, cpl.		1
3	SP1608205	Front steering cover		1
4	SP1636804	Lever, Brake - RH		1
5	SP1636813	Lever, steering adjustment		1
6	SP1608184	Steering		1
7	SP1636767	Grip, Handle Bar, cpl.		1
8	SP1636814	Rear steering cover		1
9	SP1608202	Steering adjustment Assembly, cpl.		1

Suspension

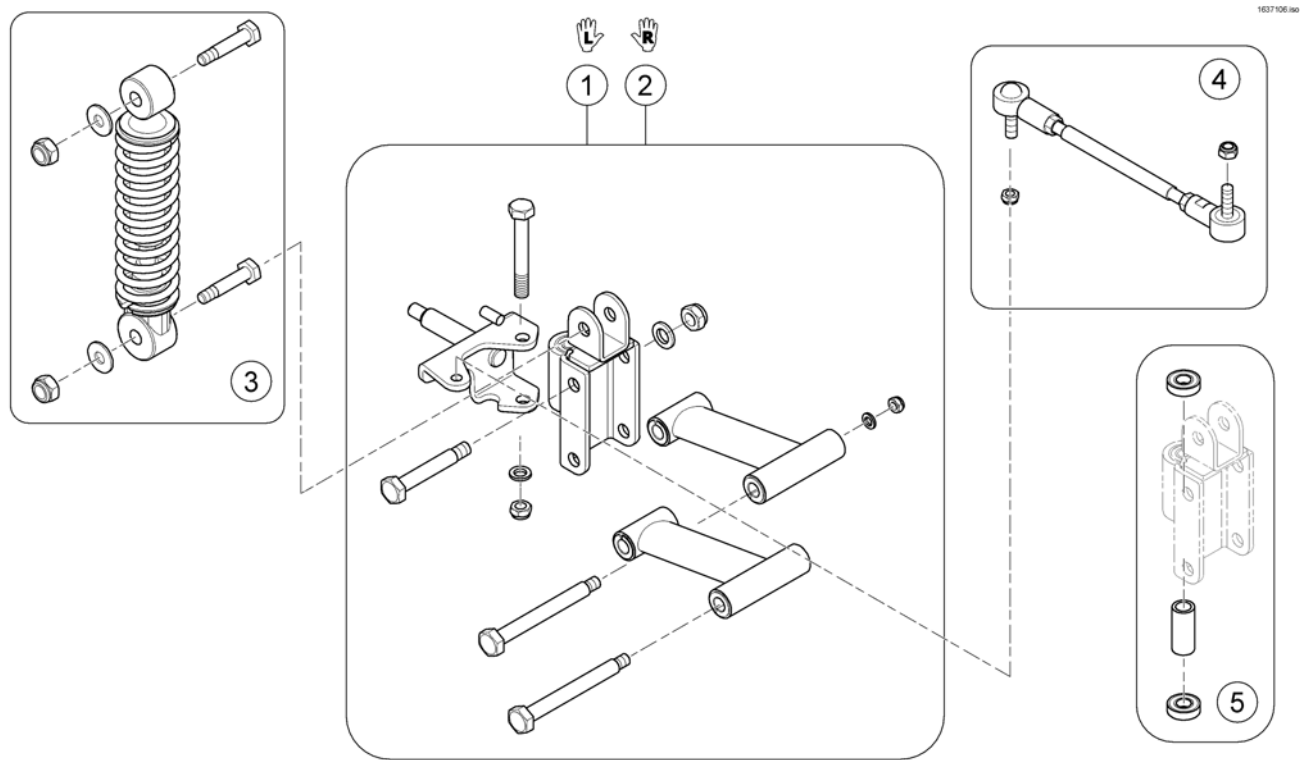

Pos.	Part number	Description	Valid from → until	Qty
1	SP1636860	Suspension - LH		1
2	SP1636862	Suspension - RH		1
3	SP1636864	Front suspension, cpl.		1
4	SP1636872	Rod Assembly, cpl.		1
5	SP1636859	Bearing Kit		1


1637083.00

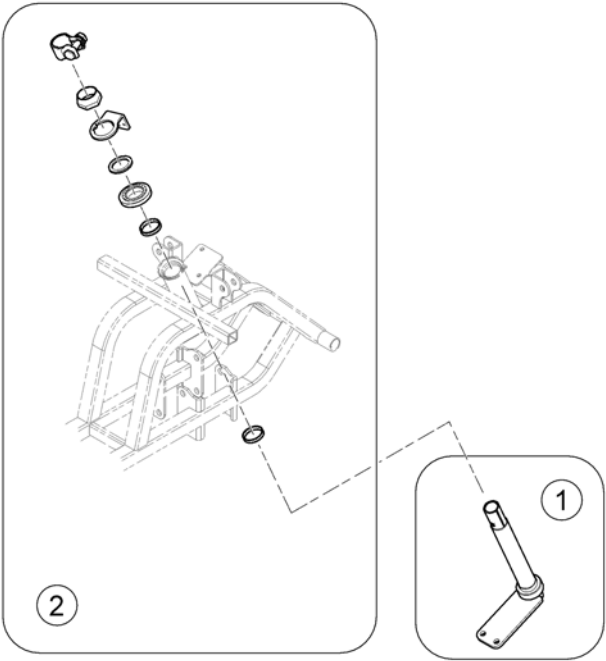

Pos.	Part number	Description	Valid from → until	Qty
1	SP1608211	Steering bar assembly		1
2	SP1636816	Bearing Kit		1


Battery & Cables

1637121/00

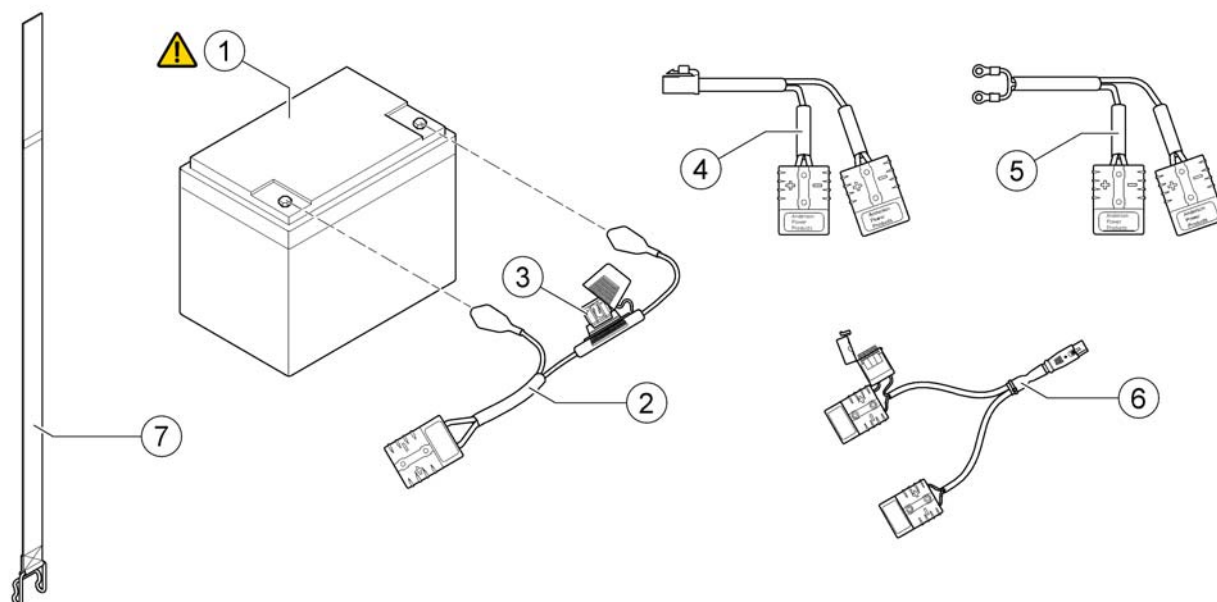

Pos.	Part number	Description	Valid from → until	Qty
!	-	Important advice! Replace batteries in pairs. Never mix: manufactures, date codes or technologies (Gel with AGM batteries)		0
1	SP1522026	Battery, 12V/75Ah C20, AGM		2
2	SP1636819	Fuse battery cable (70A) <i>PRO</i>		1
2	SP1636822	Fuse battery cable (100A) <i>ALPINE+ & ULTRA</i>		1
3	SP1636823	Fuse 70A		1
3	SP1636824	Fuse 100A		1
5	SP1637918	Wire, Battery Output - Comet ALPINE+ / ULTRA		1
4	SP1608210	Wire, Battery Output - Comet PRO		1
6	SP1641608	Off board charging cable V0		1
7	SP1521758	Battery strap		1

Chargers

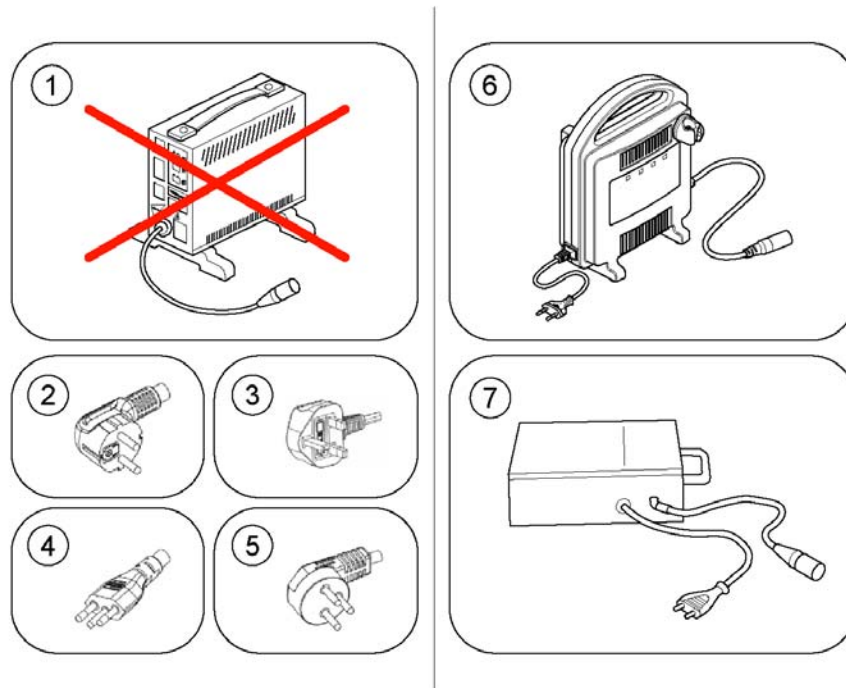


1640673.iso

Pos.	Part number	Description	Valid from → until	Qty
1	-	No longer available.		1
2	P21079	Power Cord - EU Plug (IEC 60210-C13 / CEE 7/7 Type F 'Schuko')		1
3	P21080	Power Cord - UK Plug (IEC 60320-C13 / BS 1363 Type G)		1
4	1443841	Power Cord - Swiss Plug (IEC 60320-C13 / SEV 1011 Type J)		1
5	SP1553849	Power Cord - Israel Plug (IEC 60320-C13 / SI 32 Type H)		1
6	SP1606541	Charger OptiCharge 8A - EU Plug [1343]		1
6	SP1606543	Charger OptiCharge 8A - UK Plug [1343]		1
6	SP1606544	Charger OptiCharge 10A - EU-Plug [1313]		1
6	SP1606545	Charger OptiCharge 10A - UK-Plug [1313]		1
7	5813987	Charger GI Popular 8A (IP54) - EU Plug [1388]		1

Electronic components


1637110.00

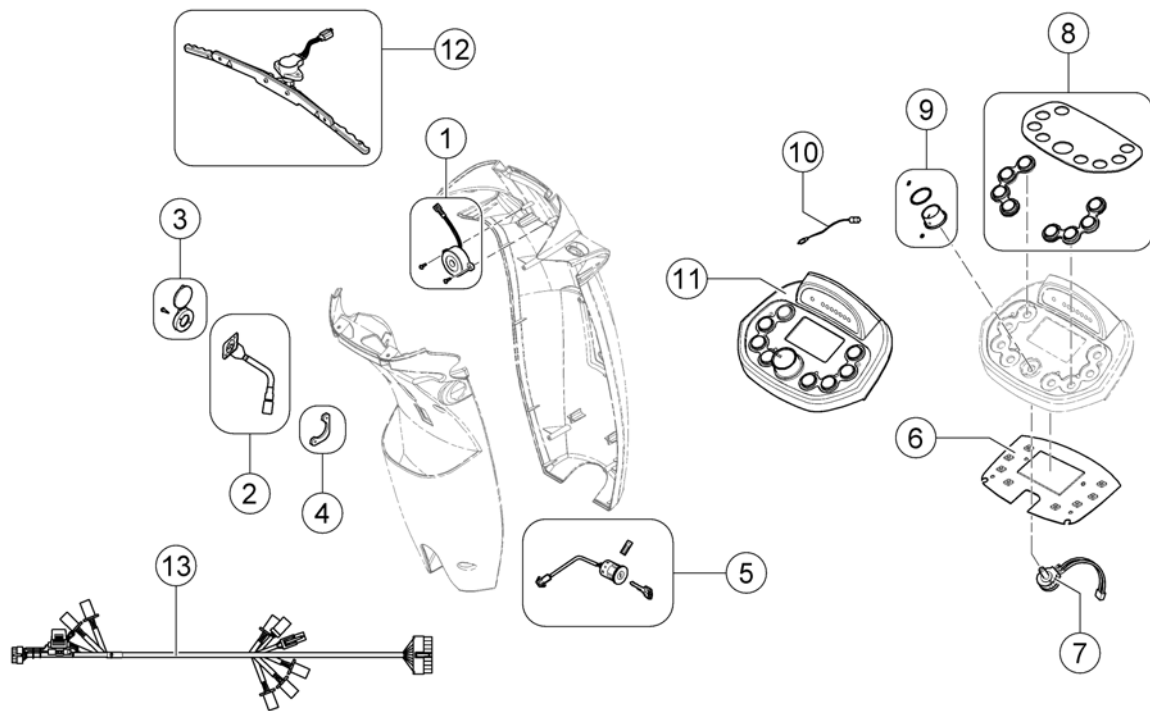

Pos.	Part number	Description	Valid from → until	Qty
1	SP1506153	Buzzer, cpl.		1
2	SP1523543	Charger-Inlett		1
3	SP1523534	Cover, Charger-Inlett		1
4	SP1523544	Bracket, Charger-Inlett		1
5	SP1505451	Key Cylinder (Standard)		1
6	SP1608208	P.C. Board LCD		1
7	SP1505462	Variable Resistor 30KOhm, cpl.		1
8	SP1637215	Display cover & rubber keys (LCD Display)		1
9	SP1608207	Knob, Speed control		1
10	SP1608204	Temperature sensor		1
11	SP1608185	Steering Upper Assembly LCD		1
12	SP1608200	Wig Wag, cpl.		1
13	SP1608212	Wire Harness - Pro		1
13	SP1608229	Wire harness - Alpine+ & Ultra		1
-	SP1521577	Fuse 10A		1

Powerbox & Limit trigger speed limit


1637122-00

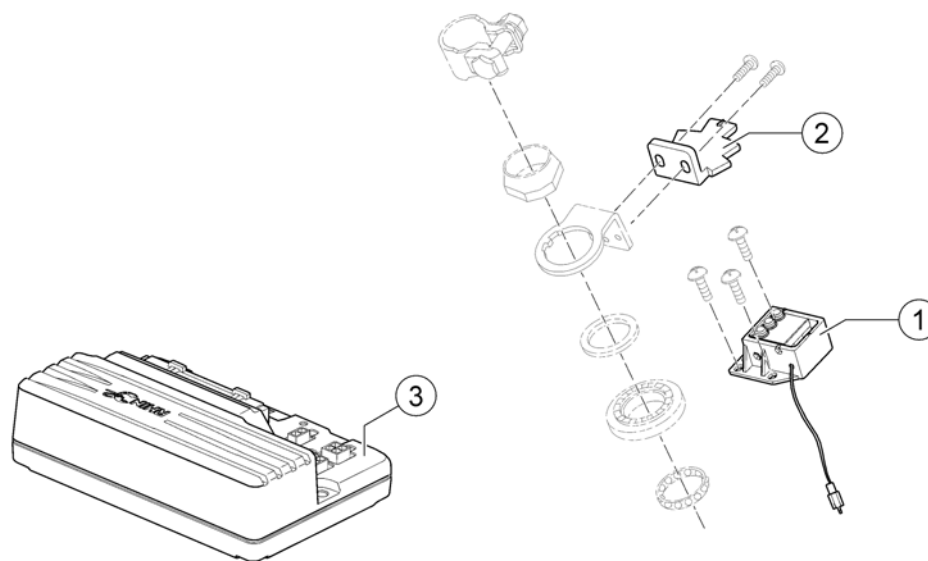

Pos.	Part number	Description	Valid from → until	Qty
-	-	For Mounting the Speed Reduction Switch, the Adjustment Gauge SP1581766 is needed!		1
-	SP1581766	Adjustment Gauge, Speed Reduction Switch		1
1	SP1556266	Speed Limit Switch Box <i>Comet Pro/Alpine+/Ultra 10 km/h</i>		1
1	SP1556281	Speed Limit Switch Box <i>Comet Pro 12.8 & 15 km/h</i>		1
2	SP1608230	Trigger, limit switch		1
3	SP1553399	Controller, Rhino2, 120A, DS120 <i>Comet Pro</i>		1
3	SP1553400	Controller, Rhino2, 160A, DS160 <i>Comet Alpine+/Ultra</i>		1

Safety


1540378.00

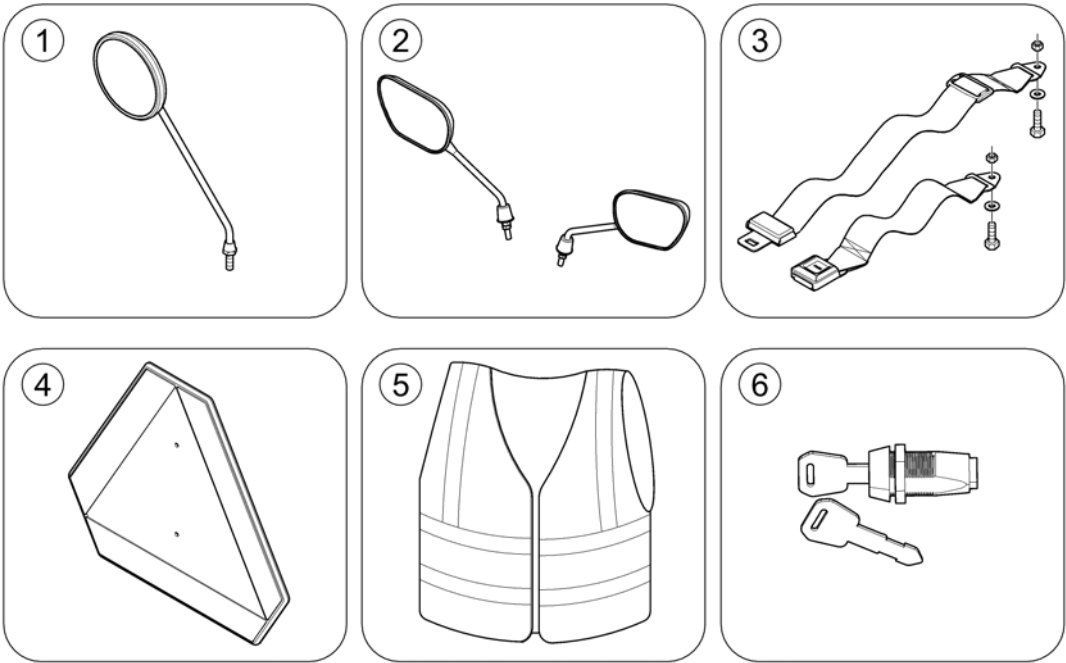

Pos.	Part number	Description	Valid from → until	Qty
1	SP1518241	Rear-view Mirror LH/RH, black		1
2	SP1637911	High-end rear mirror set		1
3	SP1518274	Lap Belt with Auto Lock Comet & Orion		1
4	-	Warning Triangle		1
5	SP1521592	Safety vest		1
6	SP1543154	Personal Key Cylinder + 2 Keys		1

Warning Triangle


OnComFig.8.160

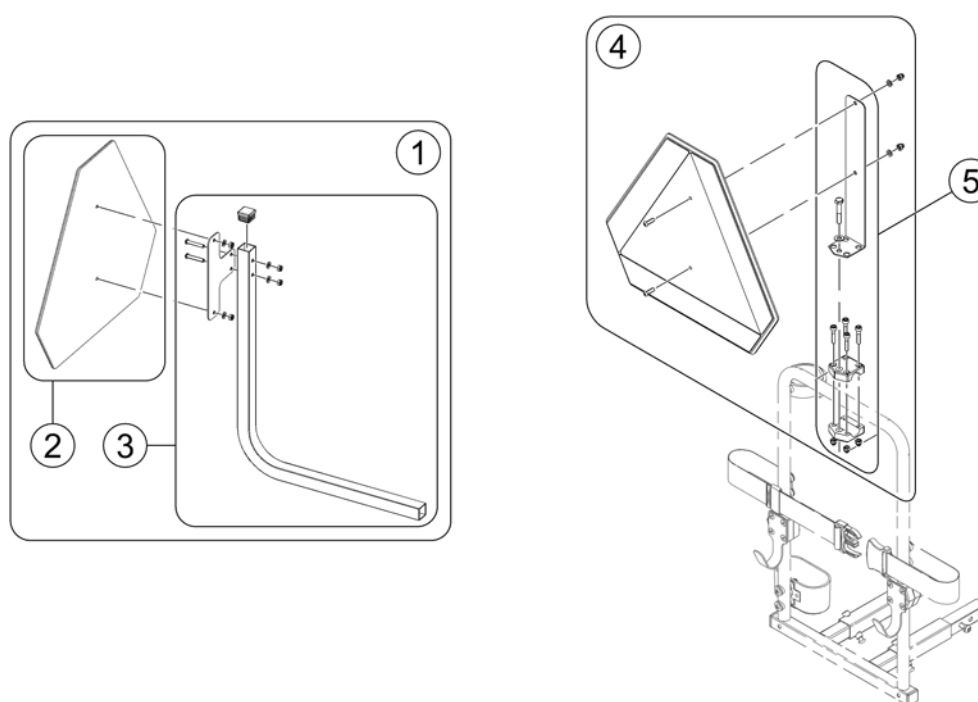

Pos.	Part number	Description	Valid from → until	Qty
1	SP1523648	Warning Triangle Scooter		1
2	1430834	Rear Marking Sign ECE-R 69		1
3	SP1526913	Holder Warning Triangel		1
4	SP1430410	Reflective Sign ECE-R 69, complete		1
5	SP1580671	Bracket, Reflective Sign, cpl.		1

Seating


1640517/00

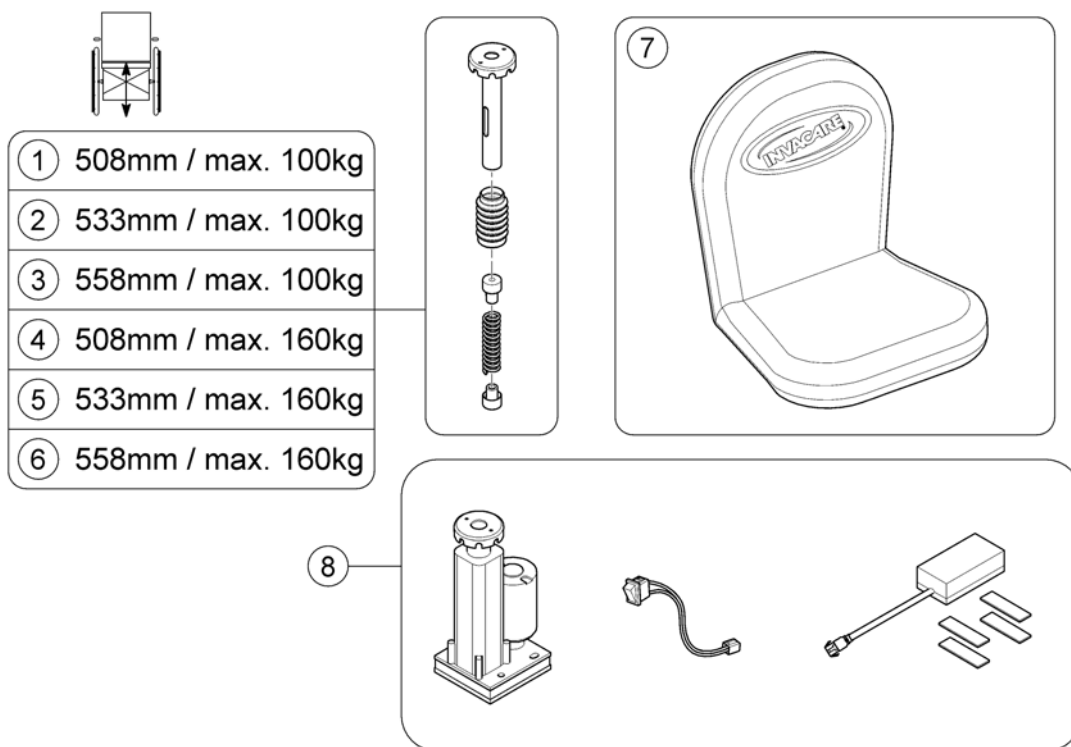

Pos.	Part number	Description	Valid from → until	Qty
1	SP1640428	Seat Suspension 226mm 201lbs		1
2	SP1640429	Seat Suspension 226mm 285lbs		1
3	SP1640430	Seat Suspension 251mm 201lbs		1
4	SP1640447	Seat Suspension 251mm 285lbs		1
5	SP1640448	Seat Suspension 276mm 201lbs		1
6	SP1640449	Seat Suspension 276mm 285lbs		1
7	SP1521684	Seat Cover Rain & Dirt protection <i>Not HD Seat!</i>		1
8	SP1641662	Powered Seat Lifter, cpl. <i>NOT Comet Ultra!</i>		1


Walking support

1540390-00

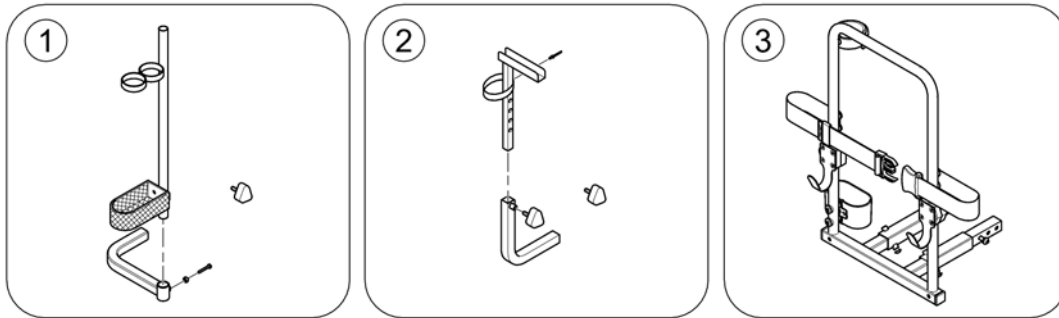


Pos.	Part number	Description	Valid from → until	Qty
1	SP1528274	Crutchholder, cpl.		1
2	SP1505728	Walking Frame Holder, cpl.		1
3	-	Rollator bracket		0

Rollator bracket

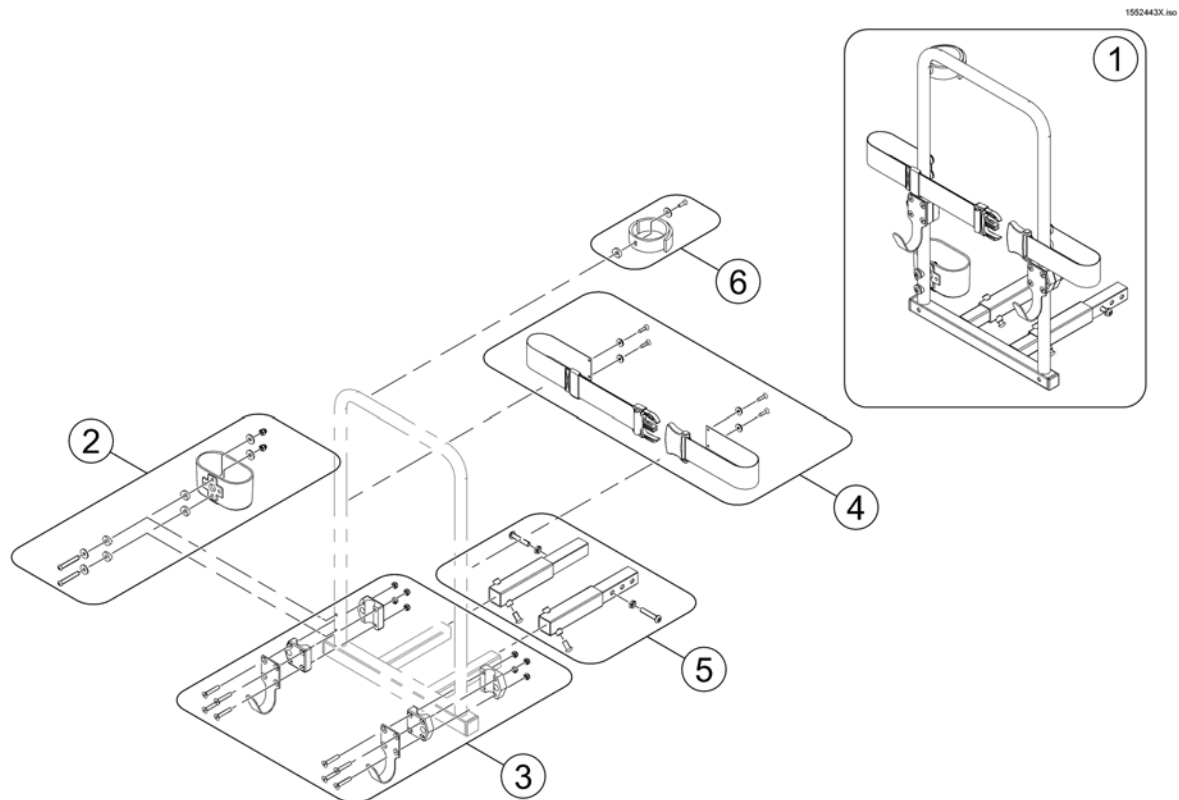


Pos.	Part number	Description	Valid from → until	Qty
1	SP1553851	Rollator Holder		1
2	SP1560465	Cane peel, cpl.		1
3	SP1553890	Rollator hooks		1
4	SP1553891	Belt, cpl.		1
5	SP1552594	Extension tubes (Comet/Orion) <i>only needed for Orion & Comet</i>		1
6	SP1640107	Strap, crutch holder, cpl.		1

Storage

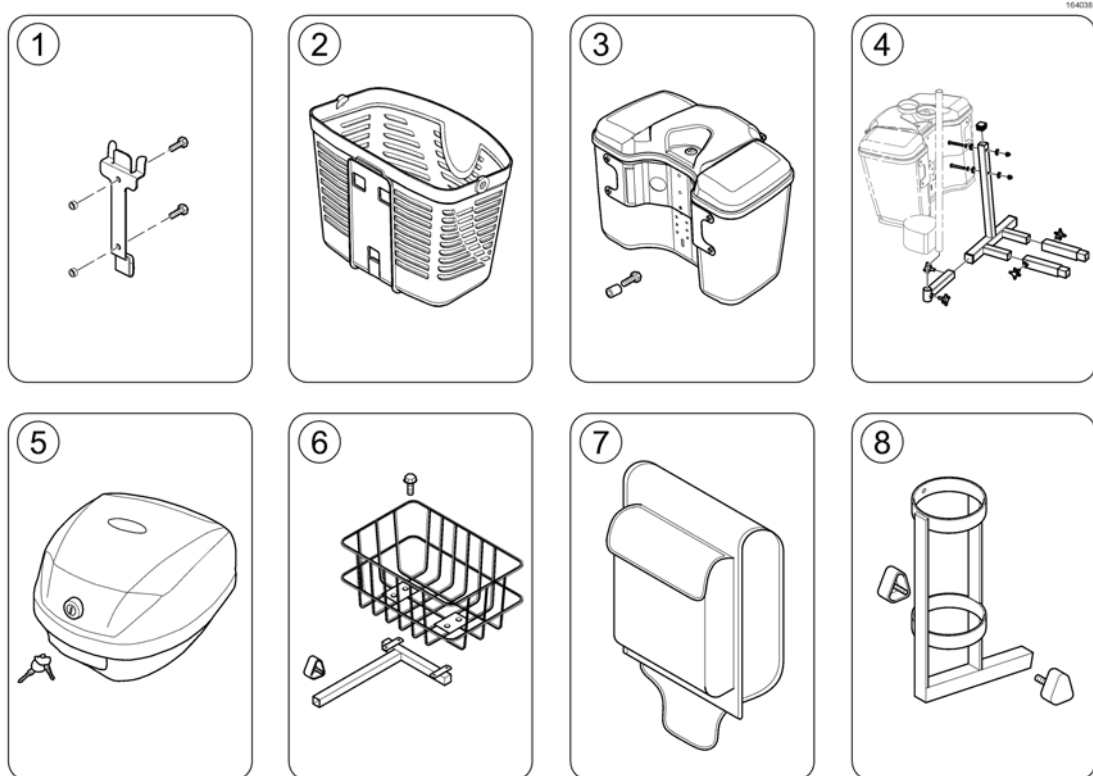

Pos.	Part number	Description	Valid from → until	Qty
1	SP1505924	Bracket, Front Basket, cpl.		1
2	SP1577430	Front Basket		1
2	SP1637915	Bigger front basket (incl. bracket) <i>Not shown / Non illustré / Nicht abgebildet</i>		1
3	SP1637910	Front Box Assembly - black		1
4	-	Storage box rear bracket		1
5	SP1637913	Lockable rear storage box (incl. bracket)		1
6	SP1505727	Rear Basket, cpl.		1
7	SP1505731	Bag, Seat Rear		1
8	SP1572971	Oxygen Bottle Holder, cpl.		1

Storage box rear bracket


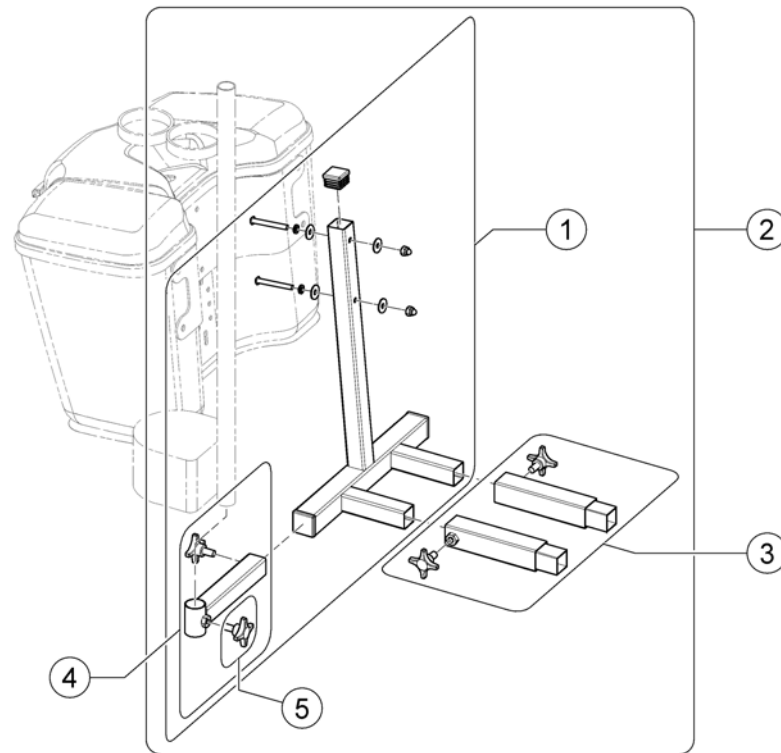


1552537X.00

Pos.	Part number	Description	Valid from → until	Qty
1	SP1552536	Rear bracket for Storage box (Leo)		1
2	SP1552537	Rear bracket for Storage Box (incl. Extension Tubes for Comet/Orion)		1
3	SP1552594	Extension tubes (Comet/Orion)		1
4	SP1553854	Crutch holder bracket		1
5	5824117	hand wheel M8x15		1


Bumpers

1640060 000

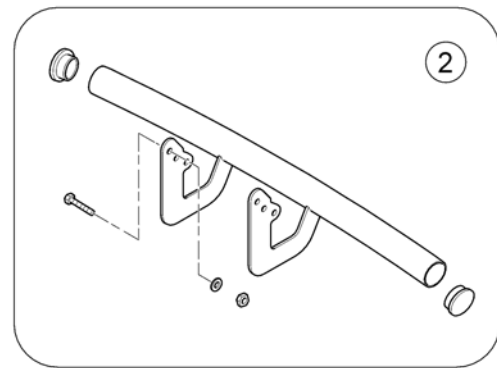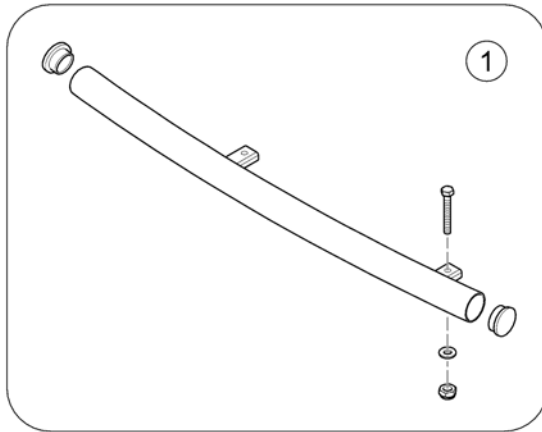

Pos.	Part number	Description	Valid from → until	Qty
1	SP1638967	Rear steel bumper		1
2	SP1528298	Front Bumper Comet 4w		1

Other Accessories

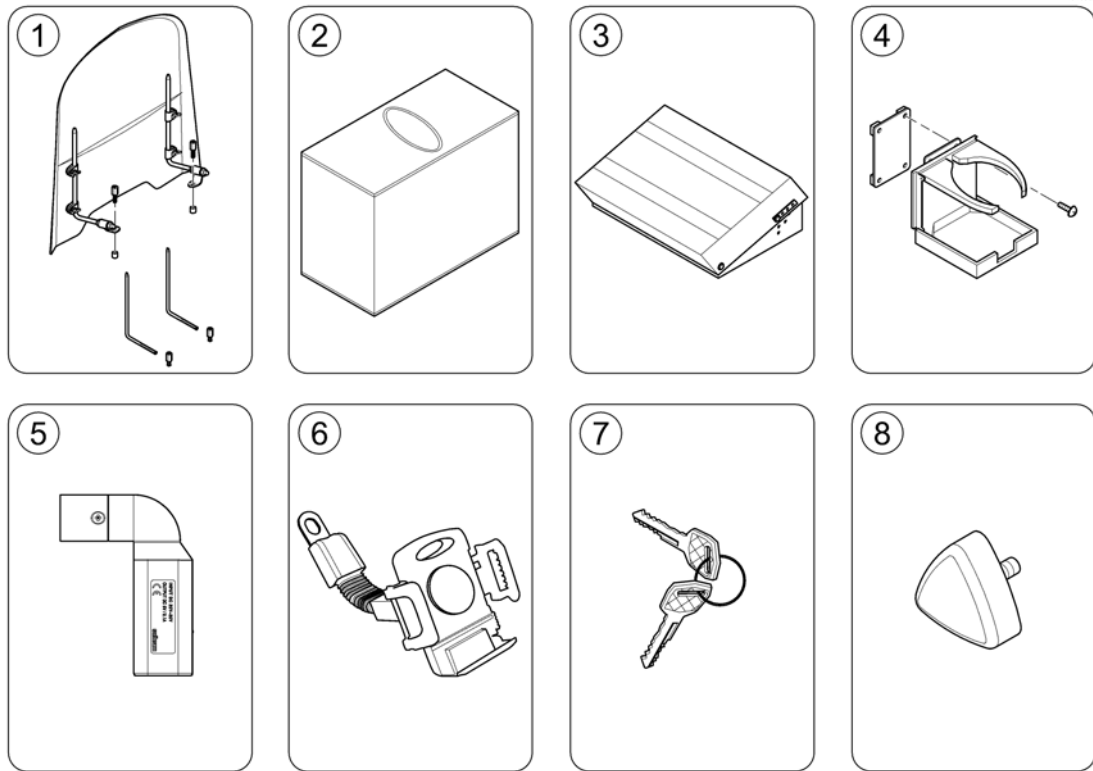

Pos.	Part number	Description	Valid from → until	Qty
1	SP1543160	Windscreen		1
2	SP1518240	Storage Cover		1
3	SP1637914	Footrest, adjustable		1
4	SP1521594	Can Holder		1
5	SP1637917	USB charger		1
6	SP1637912	Mobile phone holder		1
7	SP1530211	Key, set of 2 (Standard)		1
8	SP1507005	Star Handle		1



Yes, you can.®



Belgium & Luxemburg:

Invacare nv, Autobaan 22, B-8210 Loppem
Tel: (32) (0)50 83 10 10, Fax: (32) (0)50 83 10 11
belgium@invacare.com / www.invacare.be

Denmark:

Invacare A/S, Sdr. Ringvej 37, DK-2605 Brøndby
Tel: (45) (0)36 90 00 00, Fax: (45) (0)36 90 00 01
denmark@invacare.com / www.invacare.dk

Deutschland:

Invacare Aquatec GmbH, Alemannenstraße 10, D-88316 Isny
Tel: (49) (0)75 62 7 00 0, Fax: (49) (0)75 62 7 00 66
info@invacare-aquatec.com / www.invacare-aquatec.de

European Distributor Organisation:

Invacare, Kleiststraße 49, D-32457 Porta Westfalica
Tel: (49) (0)57 31 754 540, Fax: (49) (0)57 31 754 541
edo@invacare.com / www.invacare.eu.com

España:

Invacare SA, c/Areny s/n, Polígon Industrial de Celrà, E-17460 Celrà (Girona)
Tel: (34) (0)972 49 32 00, Fax: (34) (0)972 49 32 20
contactsp@invacare.com / www.invacare.es

France:

Invacare Poirier SAS, Route de St Roch, F-37230 Fondettes
Tel: (33) (0)2 47 62 64 66, Fax: (33) (0)2 47 42 12 24
contactfr@invacare.com / www.invacare.fr

Ireland:

Invacare Ireland Ltd, Unit 5 Seatown Business Campus, Seatown Road, Swords, County Dublin - Ireland
Tel: (353) 1 810 7084, Fax: (353) 1 810 7085
ireland@invacare.com / www.invacare.ie

Italia:

Invacare Mecc San s.r.l., Via dei Pini 62, I-36016 Thiene (VI)
Tel: (39) 0445 38 00 59, Fax: (39) 0445 38 00 34
italia@invacare.com / www.invacare.it

Nederland:

Invacare BV, Celsiusstraat 46, NL-6716 BZ Ede
Tel: (31) (0)318 695 757, Fax: (31) (0)318 695 758
nederland@invacare.com, csede@invacare.com / www.invacare.nl

Norge:

Invacare AS, Grensesvingen 9, Postboks 6230, Etterstad, N-0603 Oslo
Tel: (47) (0)22 57 95 00, Fax: (47) (0)22 57 95 01
norway@invacare.com, island@invacare.com / www.invacare.no

Österreich:

Invacare Austria GmbH, Herzog Odilostrasse 101, A-5310 Mondsee
Tel: (43) 6232 5535 0, Fax: (43) 6232 5535 4
info-austria@invacare.com / www.invacare.at

Portugal:

Invacare Lda, Rua Estrada Velha, 949, P-4465-784 Leça do Balio
Tel: (351) (0)225 1059 46/47, Fax: (351) (0)225 1057 39
portugal@invacare.com / www.invacare.pt

Sverige & Suomi:

Invacare AB, Fagerstagatan 9, S-163 91 Spånga
Tel: (46) (0)8 761 70 90, Fax: (46) (0)8 761 81 08
sweden@invacare.com, finland@invacare.com / www.invacare.se

Switzerland:

Invacare AG, Benkenstrasse 260, CH-4108 Witterswil
Tel: (41) (0)61 487 70 80, Fax: (41) (0)61 487 70 81
switzerland@invacare.com / www.invacare.ch

United Kingdom:

Invacare Limited, Pencoed Technology Park, Pencoed, Bridgend CF35 5AQ
Switchboard Tel: (44) (0)1656 776200, Fax: (44) (0)1656 776201
Customer services Tel: (44) (0)1656 776222, Fax: (44) (0)1656 776220
uk@invacare.com / www.invacare.co.uk