

**UNDERSTANDING THE NEEDS**



SHOWER & TOILET

WHEELCHAIRS & CUSHIONS

MOBILITY

BEDS & MATTRESSES

HOME / WORKING CHAIRS

TRANSFER

SCALES

EDUCATIONAL PRODUCTS

# CONTENT

Company profile .....	4
Employees .....	5-7
Staff training & Seminars .....	8-9

## Shower & Toilet

XXL-Rehab Shower commode .....	10-11
XXL-Rehab Shower Commode Wheeler .....	12-13
XXL-Rehab Shower Commode Tilt .....	14-15
XXL-Rehab Shower Commode Rise N Recline .....	16-17
XXL-Rehab Shower bench .....	18
XXL-Rehab Bedside commode .....	19

## Wheelchairs & cushions

Case story: The Childhood Shield .....	20-21
XXL-Rehab Minimaxx - Folding Wheelchair .....	22-23
XXL-Rehab Minimaxx With Push Motor .....	24-25
XXL-Rehab Manuel Wheelchair - The Eclipse .....	26-27
XXL-Rehab Manual Wheelchair - The Eclipse Tilt .....	28-29
XXL-Rehab HD electric wheelchair .....	30-31
XXL-Rehab Wheelchair cushions .....	32-33
XXL-Rehab Comfort cushion	
XXL-Rehab Anatomical cushion	
XXL-Rehab Plus cushion	

## Mobility

XXL-Rehab Stand Tall .....	34-35
XXL-Rehab Rollator King .....	36
XXL-Rehab Rollator Support .....	37
XXL-Rehab Bariatric walking frame .....	38-39
XXL-Rehab Canes .....	40-41
XXL-Rehab Walking cane	
XXL-Rehab 4-Point cane	
XXL-Rehab Crutch	

## Beds & Mattresses

XXL-Rehab Nursing Bed .....	42-43
XXL-Rehab 400 bariatric hospital bed .....	44-45
XXL-Rehab Comfort foam mattress .....	46
XXL-Rehab Low Air Loss mattress .....	47
XXL-Rehab Alternating mattress .....	48
XXL-Rehab Floor Raiser .....	50-51

## Home / Working Chairs

XXL-Rehab Patient/Dinner chair .....	52
XXL-Rehab Office Chair .....	54

## Transfer

XXL-Rehab Patient Lift and Slings .....	56-57
---	-------

## Scales

XXL-Rehab Multifunctional Scale .....	58-59
---------------------------------------	-------

## Educational Products

XXL-Rehab Bariatric Suit .....	60-61
--------------------------------	-------

Sales & delivery terms .....	62
------------------------------	----



# WELCOME TO THE WORLD OF XXL-REHAB

XXL-Rehab has specialized in supplying aids for bariatric users with the purpose of providing independency, comfort and relief.

About 20 years ago we started to import products for this patient group. We soon realized that most products were just reinforced but not redeveloped to meet the specific demands of bariatric patients.

Now we design and develop our own products. Our products are the result of many years of research in the bariatric field, knowledge obtained from congresses around the world as well as hands-on experience from bariatric users.

We have studied different body types and various mobility patterns as well as geometric studies and the law of gravity. Based on a mixture of this knowledge we design products which maintain the possibility of independency for the user.

Our products are strongly engineered and functionally designed and have a weight capacity of 325 kg. The key words for our product development are innovation and considering the needs of the users.

Our product portfolio consists of wheelchairs, shower commodes, shower benches, patient/dinner chairs, wheeled walkers, walking frames, canes, lift, slings as well as both low air loss and alternating mattresses.


Based in Denmark we supply both to the private and public sector all over the world. Our goal is to provide professional advice and high quality products within a short time frame. Our staff members are highly experienced within occupational therapy and have hands-on experience with bariatric patients.

All people are different but we have in common that we wish for quality of life. At XXL-Rehab we treat all people as were they part of our own family. For us the respect for the individual is very important.

**SIGN UP FOR THE XXL-REHAB NEWSLETTER AT**

**[www. XXL-rehab.com](http://www. XXL-rehab.com)**

and receive information about products and participation in fairs and congresses.

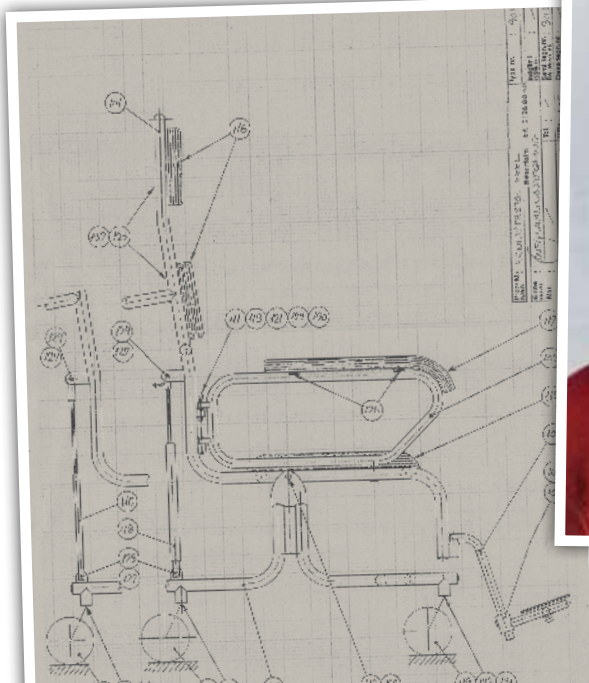


*The engineer Gert Rønne and creative director Keld Mikael Jørgensen have a unique collaboration on the development of XXL-Rehab aids for bariatric users.*



Director and head of development

*My room where I wind down.*

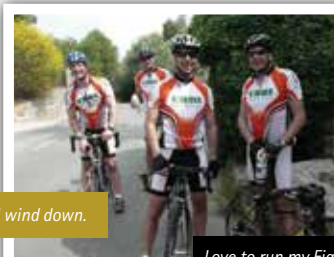

*Love to run my Fiat 850 Spider to veteran rallies.*




**EXPORT MANAGER & DIRECTOR**

**Keld Mikael Jørgensen**

**CONTACT**

E-mail: [keld@xxl-rehab.com](mailto:keld@xxl-rehab.com)

Tel.: +45 22 50 84 64

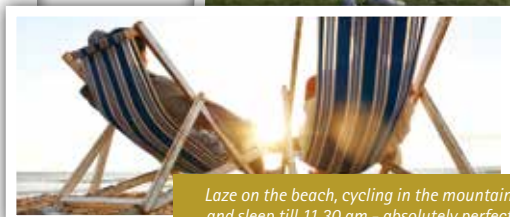

*Laze on the beach, cycling in the mountains and sleep till 11.30 am - absolutely perfect!*

*The work is also a journey in life, through which one can expand ones horizon, professionally and personally.*

Keld is the owner of XXL-Rehab and has a commercial background. Since 1995, Keld has worked with aids for physically handicapped and has much experience in the field.

Keld is passionate about making a difference for the individual, and therefore loves to be involved in developing products and concepts that can improve people's quality of life.

He prefers working in a passionate working environment where he can test new ideas, and where everyone is doing their best.

In his spare time Keld is a passionate exerciser. He loves his racing bike, he often meets with friends to speed up the wheels and is happy to participate in fun runs.



Sales consultant

**ORDERS & QUOTATIONS**

**Michael Henriksen**

**CONTACT**

E-mail: [michael@xxl-rehab.com](mailto:michael@xxl-rehab.com)

Tel.: +45 70 25 25 22



*This I would never do!*

*Here exceeded I a limit...  
On a trip in a hot air balloon.*



*Makes me really laugh.*

Michael has many years of experience in sales and has previously worked as a service manager. He works as sales consultant and is responsible for advising clients.

Michael loves to talk to customers, and it is important for him to provide a good service. Michael is very conscientious, and one can assume that his tasks are being solved.

In his private life Michael enjoys camping with his caravan and takes great pleasure in hitting the shuttlecock on the badminton court.

*It is great fun playing  
batminton!*






*OT and consultant*

**SEMINARS & STAFF TRAINING**  
**Malene Pernille Alexandrowiz**

**CONTACT**  
 E-mail: [ma@xxl-rehab.com](mailto:ma@xxl-rehab.com)  
 Tel.: +45 29 40 46 96

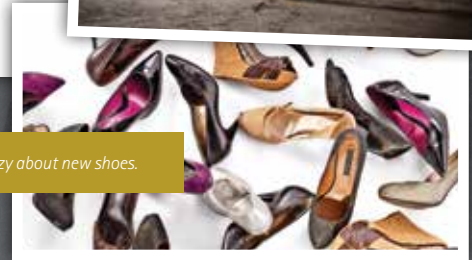

*Crazy about new shoes.*

Malene is a qualified OT and works as consultant and is responsible for teaching in XXL-Rehab. She teaches primarily within 2 focus areas where she has many years of experience, namely pain and bariatric.

Malene loves teaching and she finds it easy to combine theory and practice. In addition, Malene is responsible for XXL-Rehab being professionally updated, and participates in congresses worldwide to update her professional knowledge.

Malene has taught many places in Denmark, including Bariatric Rehabilitation Congress 2011 and 2013 in Copenhagen where she is head of the Committee. She has also taught several places abroad, including European Seating Symposium in Dublin, ISORAM in Lake Louise, CMSC in Toronto, Exposanita in Bologna, Respecta in Helsinki and Canadian Obesity Summit in Vancouver.

She has extensive experience with providing guidance to aids from previous positions within the health sector.

Musti is a trained sales consultant from Ringsted Technical College. He is the allrounder of the company and works as deputy chief of the daily operations. Additionally, he is sales consultant and responsible for planning service and storage.

Musti enjoys the daily contact with many people and brings pride in providing a good service.

Privately, he is happy to be physically active and loves playing football.

**SHIPPING & HANDLING**  
**Mustafa "Musti" Özgen**

**CONTACT**  
 E-mail: [musti@xxl-rehab.com](mailto:musti@xxl-rehab.com)  
 Tel.: +45 30 55 45 74



*Deputy chief responsible for shipping and handling*

*Recharges the batteries by fishing.*



*Enjoys playing football...*



Engineer

**PRODUCT DEVELOPER**  
Gert Rønne

Gert works as product developer in XXL-Rehab and he is the engineer behind the XXL-Rehab products for bariatric users.

Over the years, Gert has designed countless aids which have been beneficial for many people.


Gert has a unique talent for space which in combination with his patience, tenacity and empathy results in intelligent design solutions.

In his spare time Gert also enjoys developing products as well as spending time in nature.



My favorite food!

I love...  
Going hunting abroad

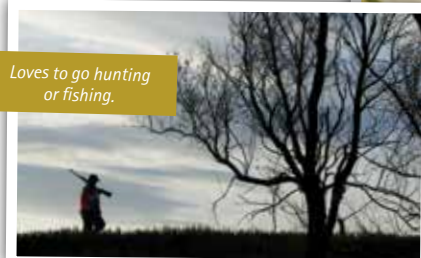

Loves to go hunting  
or fishing.



Very fond of travelling to  
Bornholm, South Africa and Paris.

## STAFF TRAINING & SEMINARS

At XXL-Rehab we have a mission to be updated with the latest knowledge within the bariatric field. For this reason, we regularly participate at international congresses and symposia focusing on research in the bariatric area.

### During the last few years we have participated at:

- ASMBS - American Society for Metabolic Bariatric Surgery, Las Vegas, USA
- CSMC, Toronto, Canada
- Annual Scientific Meeting of The Obesity Society, Washington, USA
- European Seating Symposium, Dublin, Ireland
- Emwa, Helsinki, Finland (focus on pressure ulcers and pain)

New knowledge about the needs and better understanding of the handling of bariatric patients can help us in our constant product development of our XXL-Rehab product range.

We like to share our extensive experience with practical handling of bariatric patients with our customers. It is part of our quality policy that we pass on knowledge and ensure understanding of how our products are used optimally.

Malene Alexandrowiz is the teacher of our Staff Training & Seminars. Malene is trained OT from the Occupational Therapy School of Aarhus, Denmark, and has many years of extensive experience with bariatric patients and bariatric aids.

Malene is conducting individual theme days, seminars and training courses for therapists, caregivers and nurses on a regular basis in nursing homes, hospitals, hospices etc. as well as congresses and exhibitions throughout Denmark.

### Malene has also lectured at international conferences and symposia:

- UNBC, Prince George, British Columbia, Canada, 2008
- European Seating Symposium, Dublin, Ireland, 2009
- Exposanita, Bologna, 2012
- ISORAM, Lake Louise, 2012
- Canadian Obesity Summit, Vancouver, 2013
- Respecta, Helsinki, 2013
- CSMC, Toronto, Canada, 2009 and 2013
- Bariatric Rehabilitation Congress, Denmark, 2011 and 2013





## Content of Staff Training and Bariatric Seminars

In the training Malene Alexandrowiz focus on recent research in the bariatric field and how this research can be translated into practice for therapists and specialized staff.

### Her training covers aspects of:

- Definition and review of the different degrees of obesity
- Analysis of the different body types and shapes, with presentation of our "bariatric suits"
- The different shapes makes different needs and resources
- Choice of aid, customization and deployment of the bariatric aid
- Communication, observation and interview with the bariatric patient
- Individual targets, monitoring and possible new goals focusing on rehabilitation
- Measuring of the bariatric patient
- Physical environment
- Work and working environment for care givers
- Opportunities



### WANT TO KNOW MORE..?

Send us an e-mail if you want to know more about the Staff Training & Seminars we can offer you.

[info@xxl-rehab.com](mailto:info@xxl-rehab.com)

*Malene Alexandrowiz and Keld Jørgensen demonstrating "Bariatric Suits" at a theme day in Denmark.*



# XXL REHAB®

## SHOWER COMMODE

We have aimed at designing the best shower commode on the market. It is designed for easy access, independence, safety and on the caregiver terms.

The inside width of the chair is 61 or 71 cm, but using a 5 cm high PU foam seat and 7 cm free space to the armrests allows the body mass to fall down, instead of fading out to the sides. The impossible "wide inside but small outside" is obtained. This allows the XXL-Rehab Shower Commode to pass through nearly all modern doorways.

The footrest is always the weak point when it comes to bariatric products. Normally the producer makes a short cut here, by using a standard solution. We did not only design the footrest to carry 100 kg each, we did more!

The footrest is also side/width adjustable, so access to intimate hygiene is possible and people with the fat-tissue placed on the inside of their leg can sit with legs apart (up to 110 cm) and still have foot support. The brand new footrest is even nicely covered with PU foam.

One of the unique new features is a split toilet seat, a long open hole: 44 cm and 22 cm at the widest point. This to enable a correct use of the chair, it eases access for caregivers and avoids incidents where material ends up in places it wasn't meant to end up!

The ability for bariatric users to reach far back to reach the armrest is usually not present. Therefore we have made the armrests with a length of 64 cm and let it stick out 7 cm from the seat base, ready to welcome and support the hands of mobile users. It enables the user to get into the chair by him/herself. Another option is the butterfly armrests. Butterfly armrests add more than 10 cm to the seating width.

The space between backrest and seat has a clearance of 32 cm. This gives space for a possible bulbous gluteal region, but also enables the caregivers to reach for the sling and position the user better.

- Long armrests provide easy access
- Extra seat width with butterfly armrests



Product video  
Scan the code and see  
the product in use

# STRONGLY ENGINEERED AND FUNCTIONALLY DESIGNED EXCLUSIVELY FOR BARIATRIC USERS!

## SPECIFICATIONS

Model 1      Model 2

Maximum weight capacity is	325 kg	
Seat width with standard armrests	61 cm	71 cm
Total outer width with standard armrests	68 cm	78 cm
Seat depth	56 cm	
Seat height	55 cm	
Backrest	32-52 cm	
Seat to footrest	40-42-44-46-48 cm	
Width of footrest	58-110 cm	
Pushbar from floor	100 cm	
Width between armrests	54 cm	
Height of backrest	48 cm	
Height of footrests	40-48 cm	
Distance from floor to seat underside	44.5 cm	
Total length	117 cm	
Total weight	27 kg	30 kg
Wheels (2 wheels with individual locks)	125 mm	
Seat	Pressure moulded PUR foam	
Frame	Powder-lacquered chromium molybdenum steel	
ISO - classification	ISO 09 12 03	

## ORDERING

Model	Seat width cm	Seat depth cm	Order No
XXL-Rehab Shower commode incl. standard armrests, footrests and commode bucket	61	56	0150-061-000
XXL-Rehab Shower commode incl. standard armrests, footrests and commode bucket	71	56	0150-071-000
XXL-Rehab Shower commode incl. butterfly armrests, footrests and commode bucket	61 + 14	56	0151-061-000
XXL-Rehab Shower commode incl. butterfly armrests, footrests and commode bucket	71 + 14	56	0151-071-000
Accessories			
Butterfly armrests (add 14 cm to seat width)			0150-099-012
Commode bucket			0140-099-001
Footrest right			0150-099-008
Footrest left			0150-099-009
Calf support / amputation support right			0156-099-030
Calf support / amputation support left			0156-099-031
Shower seat			0141-099-010



*Calf support / amputation support*



*Standard armrests*



*Side/width adjustable footrests*



*Butterfly armrests  
add 14 cm to seat width*



*Soft shower seat*

# XXL REHAB<sup>®</sup>

## SHOWER COMMODOE WHEELER

We have aimed at designing the best shower commode wheeler on the market. It is designed for easy access, independence, safety and on the caregiver terms. The 24" rear wheels provide more directional stability and smooth pushing.

The inside width of the chair is 61 or 71 cm, but using a 5 cm high PU foam seat and 7 cm free space to the armrests allows the body mass to fall down, instead of fading out to the sides. The impossible "wide inside but small outside" is obtained.

The footrest is always the weak point when it comes to bariatric products. Normally the producer makes a short cut here, by using a standard solution. We did not only design the footrest to carry 100 kg each, we did more!

The footrest is also side/width adjustable, so access to intimate hygiene is possible and people with the fat-tissue placed on the inside of their leg can sit with legs apart (up to 110 cm) and still have foot support. The brand new footrest is even nicely covered with PU foam.

One of the unique new features is a split toilet seat, a long open hole: 44 cm and 22 cm at the widest point. This to enable a correct use of the chair, it eases access for caregivers and avoids incidents where material ends up in places it wasn't meant to end up!

The ability for bariatric users to reach far back to reach the armrest is usually not present. Therefore we have made the armrests with a length of 64 cm and let it stick out 7 cm from the seat base, ready to welcome and support the hands of mobile users. It enables the user to get into the chair by him/herself. Another option is the butterfly armrests. Butterfly armrests add more than 10 cm to the seating width.

The space between backrest and seat has a clearance of 32 cm. This gives space for a possible bulbous gluteal region, but also enables the caregivers to reach for the sling and position the user better.

- **XXL-Rehab Shower commode wheeler with 24" rear wheels is easy to drive for the attendant**



# STRONGLY ENGINEERED AND FUNCTIONALLY DESIGNED EXCLUSIVELY FOR BARIATRIC USERS!

## SPECIFICATIONS

	Model 1	Model 2
Maximum weight capacity is	325 kg	
Seat width with standard armrests	61 cm	71 cm
Total outer width with standard armrests	68 cm	78 cm
Seat depth	56 cm	
Seat height	55 cm	
Backrest	32-52 cm	
Seat to footrest	40-42-44-46-48 cm	
Width of footrest	58-110 cm	
Pushbar from floor	100 cm	
Width between armrests	54 cm	
Height of backrest	48 cm	
Height of footrests	40-48 cm	
Distance from floor to seat underside	44.5 cm	
Total length	129 cm	
Total weight	27 kg	30 kg
Front wheels	125 mm	
Rear wheels	24"	
Seat	Pressure moulded PUR foam	
Frame	Powder-lacquered chromium molybdenum steel	
ISO - classification	ISO 09 12 03	

## ORDERING

Model	Seat width cm	Seat depth cm	Order No
XXL-Rehab Shower Commode Wheeler incl. standard armrests, footrests and commode bucket	61	56	0152-061-000
XXL-Rehab Shower Commode Wheeler incl. standard armrests, footrests and commode bucket	71	56	0152-071-000
XXL-Rehab Shower Commode Wheeler incl. butterfly armrests, footrests and commode bucket	61 + 14	56	0153-061-000
XXL-Rehab Shower Commode Wheeler incl. butterfly armrests, footrests and commode bucket	71 + 14	56	0153-071-000
Accessories			
Butterfly armrests (add 14 cm to seat width)			0150-099-012
Commode bucket			0140-099-001
Footrest right			0150-099-008
Footrest left			0150-099-009
Shower seat			0141-099-010
Calf support / amputation support right			0156-099-030
Calf support / amputation support left			0156-099-031



*Calf support / amputation support*



*Standard armrests*



*Side/width adjustable footrests*



*Butterfly armrests  
add 14 cm to seat width*



*Soft shower seat*

# XXL REHAB<sup>®</sup>

## SHOWER COMMODOE TILT

The XXL-Rehab Shower Commode Electric Tilt has a 32 degree tilt position and a weight capacity of 325 kg Safe Working Load. This shower commode is designed to provide a comfortable alternative showering system in instances where the prone position might cause discomfort or distress to the user. It may also be used in circumstances of limited space providing a bathing solution where other units may not have room for operation.

### Making tasks easier and safer

The XXL-Rehab Shower Commode Tilt provides better access for the caregiver in terms of hygiene, skin and tissue care as well as nursing/dressing feet in standing position, allowing the caregiver to work in a safe and ergonomically sound way.

### Improving care, efficiency and communication

This shower commode raises the efficiency of care because it enables a single caregiver to perform the whole showering procedure, as well as a wide range of everyday hygiene tasks. The high level of support provided by the shower commode results in streamlined routines that offer considerable time savings compared with traditional methods. It also enables the caregiver and the user to maintain eye contact for better communication.

### Suitable for

The XXL-Rehab Shower Commode Tilt is suitable for patients with reduced muscle tone, pressure ulcers, spasm, involuntary movement, amputations or stroke patients.

- Electric tilt up to 32°
- The high backrest and sturdy armrests provide optimum user support and security
- The reclined position gives full access for the caregiver to most parts of the body and is very comfortable for the user



Product video  
Scan the code and see  
the product in use



With standard footrests

## STRONGLY ENGINEERED AND FUNCTIONALLY DESIGNED EXCLUSIVELY FOR BARIATRIC USERS!

### SPECIFICATIONS

Maximum weight capacity is	325 kg
Seat depth adjustable	42 / 48 / 54 cm
Total length (lowest position)	140 cm
Total width	74 cm
Seat width	66 cm
Seat height	51-69 cm
Backrest	24-53 cm
Width of footrest	58-110 cm
Height of footrests	40-48 cm
Electric tilt	0-32°
Width between standard armrests	61 cm
Width between butterfly armrests	76 cm
Width toilet opening	Diameter 24 cm, 35 cm long
Distance from floor to seat underside	44.5 cm
Pushbar from floor	100 cm
Seat to footrest	39-42-45-48 cm
Height seat to armrest	22 cm
Weight	39 kg
Folding armrests	No
Swing-away armrests	Yes
Swing-away footrests	Yes
Removable footrests	Yes
Height adjustable footrests	Yes
Headrest	Yes
4 wheels with individual locks	Yes
Control	Via the control panel
Capacity up/down fully charged battery	Approx. 20 times
Turning radius	158 cm
Frame	Powder lacquered chromium molybdenum steel
Wheels diameter	150 mm
Seat / back / armrests	Polyurethane
IP - classification	IP44
ISO - classification	ISO 09 12 03 - 23

### ORDERING

Model	Seat width cm	Seat depth cm	Order No
XXL-Rehab Shower Commode Tilt incl. standard footrests and charger	66	42 / 48 / 54	0155-066-000
<b>Accessories</b>			
Butterfly armrests (add 14 cm to seat width)			0150-099-012
Headrest adjustable			0156-099-020
Commode seat, 2 part			0150-099-020
Commode bucket			0140-099-001
Footrest right			0150-099-008
Footrest left			0150-099-009
Calf support / amputation support right			0156-099-030
Calf support / amputation support left			0156-099-031



*The handset gives the caregiver control over battery-powered functions*



*Commode bucket and seat*



*Headrest*

# XXL REHAB<sup>®</sup>

## SHOWER COMMODORE RISE N RECLINE

The Shower Commode Rise N Recline lifts the user from 55 to 100 cm and has a weight capacity of 325 kg Safe Working Load. It is the world's first shower commode for bariatric users that automatically can reach a 20 degree tilt position by pushing only one button. The chair tilts slowly while lifting the user and reaching a 20 degree tilt in the highest position.

### A truly ergonomic multi-purpose hygiene chair

The XXL-Rehab Shower Commode Rise N Recline was developed in response to the growing trend towards showering in bariatric care and skincare as traditionally used methods are uncomfortable for the user and impose strain on the caregiver. The innovative ergonomic design of the shower commode is highly versatile and can also support better routines for washing/showering, hair washing, wound care, toileting etc.



### Enhancing lives

The XXL-Rehab Shower Commode Rise N Recline can enhance the quality of life by making assisted showering and other hygiene routines not only safer and more comfortable, but also a more dignified experience.

### Making tasks easier and safer

Ergonomic design, battery-powered assistance and the Care Raiser function removes the stress and strain from showering and other everyday hygiene routines, allowing the caregiver to work in a safe and ergonomically sound way.

### Improving care, efficiency and communication

This shower commode raises the efficiency of care because it enables a single caregiver to perform the whole showering procedure, as well as a wide range of everyday hygiene tasks. The high level of support provided by the shower commode results in streamlined routines that offer considerable time savings compared with traditional methods. It also enables the caregiver and the user to maintain eye contact for better communication.

- The Care Raiser function is unique, as it lifts the lower part of the user's body to enable easy and discreet access for sensitive hygiene tasks.
- The high backrest and sturdy armrests provide optimum user support and security.
- The reclined position gives full access for the caregiver to most parts of the body and is very comfortable for the user.



*In lowest position with standard footrests*

Product video  
Scan the code and see  
the product in use



## STRONGLY ENGINEERED AND FUNCTIONALLY DESIGNED EXCLUSIVELY FOR BARIATRIC USERS!

### SPECIFICATIONS

Maximum weight capacity	325 kg
Seat depth adjustable	42 / 48 / 54 cm
Total length (lowest position)	140 cm
Total width	74 cm
Seat width	66 cm
Seat height	51-69 cm
Electric tilt	0-20°
Tilt of seat independent from height adjustment	Yes
Width between standard armrests	61 cm
Width between butterfly armrests	76 cm
Width toilet opening	Diameter 24 cm, 35 cm long
Seat height electric adjustable	55-100 cm
Distance from floor to seat underside	44.5 cm
Pushbar from floor	100 cm
Seat to footrest	39-42-45-48 cm
Height seat to armrest	22 cm
Weight	46 kg
Swing-away armrests	Yes
Swing-away footrests	Yes
Removable footrests	Yes
Height adjustable footrests	Yes
Headrest	Yes
4 wheels with individual locks	Yes
Control	Via the control panel
Capacity up/down fully charged battery	Approx. 15 times
Turning radius	156 cm
Frame	Powder lacquered chromium molybdenum steel
Wheels	Diameter 150 mm
Seat / back / armrests	Polyurethane
IP - classification	IP44
ISO - classification	ISO 09 12 03 - 23

### ORDERING


Model	Seat width cm	Seat depth cm	Order No
XXL-Rehab Shower Commode Rise N Recline incl. standard footrests and charger	66	42 / 48 / 54	0156-066-000
<b>Accessories</b>			
Butterfly armrests (add 14 cm to seat width)			0150-099-012
Headrest adjustable			0156-099-020
Commode seat, 2 part			0150-099-020
Commode bucket			0140-099-001
Footrest right			0150-099-008
Footrest left			0150-099-009
Calf support / amputation support right			0156-099-030
Calf support / amputation support left			0156-099-031



*The handset gives the caregiver control over battery-powered functions*



*Commode bucket and seat*

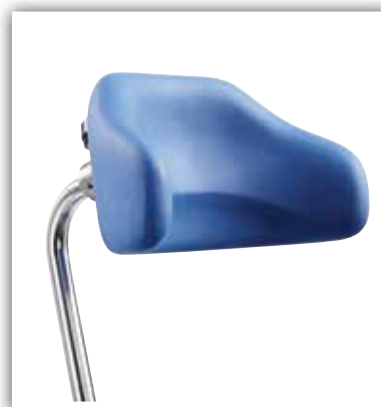

*Headrest*

# XXL REHAB<sup>®</sup>

## SHOWER BENCH

STRONGLY ENGINEERED AND FUNCTIONALLY DESIGNED  
EXCLUSIVELY FOR BARIATRIC USERS!

The armrests are constructed so they go beyond the seat. This makes it easy both to get a good grip when a person wants to sit down but also to get up again. We have designed the armrests to bend out like a butterfly, so the body doesn't get squeezed, getting in and out the shower bench. The butterfly armrests of the XXL-Rehab Shower bench also provide much better functionality for the user, compared to other shower benches with straight and short armrests where it is almost impossible for a bariatric user to get a good grip.

The height of the shower bench is adjustable from 44 to 59 cm. It is very important to adjust the shower bench to the right height to have success getting in and out of the bench. If the shower bench is too high for the user, the feet will lift off the ground, making it hard to get seated in the shower bench. If the shower bench is too low, the user will have trouble getting up.



It is often a problem for bariatric users to be seated comfortably. Sometimes the seat depth is too short, sometimes too deep and with no back support. We have designed the seat to be individually adjusted to provide a comfortable seating. The backrest can be moved +/- 6 cm and the bended tubing assures that the right comfort can be found.

It is easy to clean the XXL-Rehab Shower bench with its removable seat pad. No tools are needed.

It is also made easy to both handle and move the shower bench from place to place. In the construction of the shower bench we have only used high-tech lightweight tubing, to provide the highest strength at the lowest weight.

- Adjustable seat height
- Adjustable seat depth

### ORDERING

Model	Seat width cm	Seat depth cm	Order No
XXL-Rehab Shower Bench	61	40-52	0141-061-000
XXL-Rehab Shower Bench	71	40-52	0141-071-000

### SPECIFICATIONS

Model 1      Model 2

Maximum weight capacity is	325 kg	
Seat width	61 cm	71 cm
Total length	80-101 cm	
Seat depth	40-52 cm	
Seat height	44-59 cm	
Backrest height	37 cm	
Distance from floor to seat underside	33-48 cm	
Total outer width of front legs	69 cm	79 cm
Total height	79-94 cm	
Total weight	15 kg	16 kg
Total outer width on armrests	73 cm	83 cm
Seat / back / armrests	Polyurethan	
Frame	Powder-lacquered chromium molybdenum steel	
ISO - classification	ISO 09 33 03	

# XXL REHAB®

## BEDSIDE COMMUNE

STRONGLY ENGINEERED AND FUNCTIONALLY DESIGNED  
EXCLUSIVELY FOR BARIATRIC USERS!

The armrests are constructed so they go beyond the seat. This makes it easy both to get a good grip when a person wants to sit down but also to get up again. We have designed the armrests to bend out like a butterfly, so the body doesn't get squeezed, getting in and out the chair. The butterfly armrests of the XXL-Rehab Bedside commode also provide much better functionality for the user, compared to other chairs with straight and short armrests where it is almost impossible for a bariatric user to get a good grip.

The height of the chair is adjustable from 44 to 59 cm. It is very important to adjust the chair to the right height to have success getting in and out of the chair. If the chair is too high for the user, the feet will lift off the ground, making it hard to get seated in the chair. If the chair is too low, the user will have trouble getting up.

It is often a problem for bariatric users to be seated comfortably. Sometimes the seat depth is too short, sometimes too deep and with no back support. We have designed the seat to be individually adjusted to provide a comfortable seating. The backrest can be moved +/- 6 cm and the bended tubing assures that the right comfort can be found.

It is easy to clean the XXL-Rehab Bedside commode with its removable seat pad. No tools are needed and the bedpan is redesigned for a smoothly hygienic cleaning operation.

It is also made easy to both handle and move the chair from place to place. In the construction of the chair we have only used high-tech lightweight tubing, to provide the highest strength at the lowest weight.

- Adjustable seat height
- Adjustable seat depth



### ORDERING

Model	Seat width cm	Seat depth cm	Order No
XXL-Rehab Bedside commode	61	40-52	0140-061-000
XXL-Rehab Bedside commode	71	40-52	0140-071-000
Accessories			
Commode bucket			0140-099-001

### SPECIFICATIONS

Model 1      Model 2

Maximum weight capacity is	325 kg	
Seat width	61 cm	71 cm
Total length	80-101 cm	
Seat depth	40-52 cm	
Seat height	44-59 cm	
Backrest height	37 cm	
Distance from floor to seat underside	33-48 cm	
Distance from floor to bucket underside	21-36 cm	
Total outer width of front legs	69 cm	79 cm
Total height	79-94 cm	
Total weight	14 kg	15 kg
Total outer width on armrests	73 cm	83 cm
Seat / back / armrests	Polyurethan	
Frame	Powder-lacquered chromium molybdenum steel	
ISO - classification	ISO 09 33 03	

# The Childhood Shield

## Every body tells a story

Henrik Fenstoft is born on a summer day in 1969 in Odense, Denmark. As a child, Henrik's weight is normal; his birth weight is only 2550 kilogram. As a teenager he starts to gain weight and at the age of 16 he weighs 125 kg. When he turns 27 his weight reaches 240 kg. In his mid thirties Henrik's weight exceeds 300 kg and he starts to receive injections with the male sex hormone testosterone which put a stop to the weight gain. Until then numerous consultations with dieticians and diet plans have had no effect. Henrik is always hungry and never feels full.

At the age of 40 Henrik gets diagnosed with type 2 diabetes and over a short period of time he gains 56 kg and reaches a weight of 364 kg. Henrik is unhappy, he gets skin infections, his legs cannot support the weight of his body and he spends his life in his bed and his XXL-Rehab Minimaxx wheelchair.

The doctors demand he must lose weight to get a gastric bypass surgery and be able to walk 50 m. A new dietician is appointed to Henrik and she has a completely different approach to diet plans and how to lose weight. She focuses on the small changes to the diet and on replacing the most fattening products with healthier options. The dietician and Henrik start cooperating around Henrik's diet; soon Henrik is sole responsible and his weight loss is steady. His weight drops to 291 kg, he feels much better and he starts feeling full again. He is still lying in his bed but he is happy.

Having thought about it, Henrik changes his mind about having a gastric bypass surgery. First of all the side effects are too many. Secondly, the weight loss is going well and he has come to realise that there could be a reason to why he has eaten so much. He sets a new goal: to be able to get out of bed and to walk – by himself. In short: to regain control over his life.



Henrik is granted double up on rehabilitation to achieve his goal. Little by little he is able to walk again by using the XXL-Rehab Stand Tall. More improvements follow. Henrik starts taking showers on his own, using a shower commode, his mobility improves – he moves around his house by using a rollator.

Henrik is still very interested in food – he wants to find the reason why he has always eaten so much. Conversations with a psychologist help him find the answer. He realises that he has had an eating disorder since his days at school. The many kilos he gained as a teenager has been an attempt to create a shield. A shield that could protect him from being bullied by the other children at school, a shield that could help him through all the defeats that followed. And no matter how hard he tried, he never received his father's recognition. Food was a comfort.

Now food is no longer a comfort. He is more confident, sets small goals of weight loss that he can achieve so he can get the feeling of small successes. His weight is 251 kg and he is wants to lose more weight to become totally independent – and he will keep on fighting to get there.





# XXL REHAB<sup>®</sup>

## MINIMAXX - FOLDING WHEELCHAIR

The XXL-Rehab Minimaxx folds up easily and is easy to bring along. The folding system is very easy to activate, and gives at the same time great support to the seat. The design of the folding system makes the chair rigid – when unfolded the chair is also much more stable than normal foldable chairs.

STW (Stiffness To Weight) was our goal. We used very light tubing technology as used on high end MTB and road bikes. This Minimaxx only weighs approx. 9% of its weight capacity and still provides a smooth drive thanks to the rigid folding system and light tubing.

The XXL-Rehab Minimaxx drives very well due to the position of the rear and front wheels. It is designed so the rear wheels carry most of the user's weight making it much easier for the user to push. The castor wheels are further in front as on standard chairs. This prevents the chair from tipping.

The low seat height makes it easy to access the chair and still propel it.

The push handles are designed so the attendant can easily use his/her body to turn and push the chair. The padded handles are at an angle that puts as little strain to the wrists and shoulders as possible.

The XXL-Rehab Minimaxx has disc brakes as an additional option. Little effort is required to brake the chair even with a full load. To the disc brake is also added a parking brake and the chair also has a wheels lock. Another option is the butterfly armrests. Butterfly armrests add more than 10 cm to the seating width.

The footrests are designed to carry 100 kg each! The footrests are also side/width adjustable. This enables users with legs aligned or with a wide leg spread to use the chair.

The very long armrests make it easy for the user to get in and out of the chair.

- Folding rigid frame
- Super easy to push
- Different armrests available, see page 25



Product video  
Scan the code and see  
the product in use

# STRONGLY ENGINEERED AND FUNCTIONALLY DESIGNED EXCLUSIVELY FOR BARIATRIC USERS!

## SPECIFICATIONS

Maximum weight capacity is	325 kg
Seat width	56 / 61 / 66 / 71 cm
Total width	Seat width + 22 cm
Total width folded	35 cm
Total height	103 cm
Total length without footrests	70 cm
Seat depth	46 cm
Seat height	43 cm
Backrest height	51 cm
Weight	Approx. 27 kg
ISO – classification	ISO 12 22 03 – 08

## ORDERING

Model	Seat width cm	Seat depth cm	Order No
XXL-Rehab Minimaxx Standard incl. footrests	56	46	0100-056-000
XXL-Rehab Minimaxx Standard incl. footrests	61	46	0100-061-000
XXL-Rehab Minimaxx Standard incl. footrests	66	46	0100-066-000
XXL-Rehab Minimaxx Standard incl. footrests	71	46	0100-071-000
XXL-Rehab Minimaxx with disc brakes, incl. footrests	56	46	0109-056-000
XXL-Rehab Minimaxx with disc brakes, incl. footrests	61	46	0109-061-000
XXL-Rehab Minimaxx with disc brakes, incl. footrests	66	46	0109-066-000
XXL-Rehab Minimaxx with disc brakes, incl. footrests	71	46	0109-071-000
Accessories			
Butterfly armrests, pair			0100-099-005
Sport armrests, pair			0100-099-006
Table			0100-099-001
Cane holder for 1 cane			0100-099-022
IV holder and O2 holder, diameter 10 cm			0100-099-020
Calf support / amputation support, right			0100-099-015
Calf support / amputation support, left			0100-099-016
Footrest swing-away, right			0100-099-010
Footrest swing-away, left			0100-099-011



Disc brakes




Table



IV holder and O2 holder



Hand brake operated by attendant

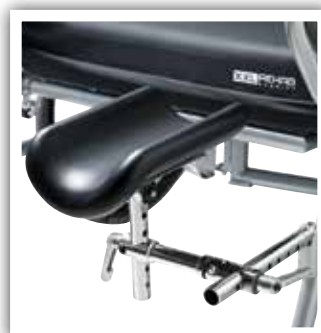

Calf support / amputation support

# XXL REHAB®

## MINIMAXX WITH PUSH MOTOR

The XXL-Rehab Minimaxx With Push Motor is a strong power pack that assists the attendant to push the chair with no effort. It is now easy to move a bariatric patient around. With the XXL-Rehab Minimaxx With Push Motor the caregiver avoids sore shoulders and stress on wrists.

Even on slopping surface up to 5 % the XXL-Rehab Minimaxx With Push Motor proudly helps over you over the top and going down you find the security of our disc braking system. If an immediate stop is needed just pull the lever and the wheelchair stops at once.

The push motor is easy to operate. The control panel is placed on the handle and offers Drive and Reverse mode as well as speed control from 1-5 km/h.

As for the construction of the wheelchair STW (Stiffness To Weight) was our goal. We used very light tubing technology as used on high end MTB and road bikes. This XXL-Rehab Minimaxx only weighs approx. 9% of its weight capacity and still provides a smooth drive thanks to the light tubing.

The castor wheels are further in front as on standard chairs. This prevents the chair from tipping.

The very long armrests make it easy for the user to get in and out of the chair. The low seat height makes it easy to access the chair.

The footrests are designed to carry 100 kg each! The footrests are also side/width adjustable. This enables users with legs aligned or with a wide leg spread to use the chair.

- Push motor is easy to operate
- Different armrests available





**STRONGLY ENGINEERED AND FUNCTIONALLY DESIGNED  
EXCLUSIVELY FOR BARIATRIC USERS!**

**SPECIFICATIONS**

Maximum weight capacity is	325 kg
Seat width	56 / 61 / 66 / 71 cm
Total width	Seat width + 22 cm
Total height	103 cm
Total length without footrests	70 cm
Seat depth	46 cm
Seat height	43 cm
Backrest height	51 cm
Weight	Approx. 27 kg
Weight push motor	23 kg
Weight battery	9 kg
Battery	12V 12AH x 2
Motor	24V 450W 4.5 amps
Tire size	50 mm x 200 mm (Solid Tire) x 2
Charger capacity	Off Board (1.8A)
Range per charge	4.5 km
Max. speed	5 km/h
Gradient	5°
ISO – classification	ISO 12 23 12 - 02

**ORDERING**

Model	Seat width cm	Seat depth cm	Order No
XXL-Rehab Minimaxx With Push Motor, incl. disc brakes and footrests	56	46	0110-056-000
XXL-Rehab Minimaxx With Push Motor, incl. disc brakes and footrests	61	46	0110-061-000
XXL-Rehab Minimaxx With Push Motor, incl. disc brakes and footrests	66	46	0110-066-000
XXL-Rehab Minimaxx With Push Motor, incl. disc brakes and footrests	71	46	0110-071-000
Accessories			
Butterfly armrests, pair			0100-099-005
Sport armrests, pair			0100-099-006
Table			0100-099-001
Cane holder for 1 cane			0100-099-022
IV holder and O2 holder, diameter 10 cm			0100-099-020
Calf support / amputation support, right			0100-099-015
Calf support / amputation support, left			0100-099-016
Footrest swing-away, right			0100-099-010
Footrest swing-away, left			0100-099-011

*Push motor*



*Disc brakes*



*Standard armrests*



*Butterfly armrests*



*Sport armrests*



*Sport armrests, side view*

## MANUEL WHEELCHAIR - THE ECLIPSE

The construction and adjustment possibilities meet the needs from various bariatric bodytypes, making it the best for this type of users.

The steel frame gives the XXL-Rehab Eclipse wheelchair maximum strength and rigedness, and therefore also makes it a long lasting wheelchair.

The frame is designed so most of the userweight is carried on the rear wheels. The wheels are especially made for this wheelchair; they are extra hard and solid making propulsion and steering unbelievably easy and with very little loss of energy!

The seat pan is also solid – because anything else would be hazardous! Various medical conditions demand a solid seatpan (fractures, hemiplegia, lateralshifts etc) but also giving a bariatric user a solid seat pan facilitates a good posture and activeness in the chair.

The XXL-Rehab Eclipse Wheelchair has adjustable seat depth, seat height, seat to floor angle and seat to back angle. All of this makes it easy to adjust to any user! The adjustment possibilities give the user a chance of staying active in propelling the chair, to get in and out of the chair and maybe also walk the chair.

The back is constructed so users with a protruding behind can get good support to their lumbar spine, a huge problem rarely met by any other chair!

Front wheels are placed in front of the seat pan to ensure an anterior anti-tip, which is very important taking into consideration that most bariatric users have a lot of weight placed in front of them. The anterior anti-tip makes it possible for the user to get in and out of the wheelchair without the risk of tipping. The XXL-Rehab Eclipse Wheelchair has disc brakes as an additional option.

Different types of front rigging complete the adjustment possibilities. The chair can be used with 3 different kinds of cushions.

- Can be reinforced to capacity of 450 kg
- Possible to adjust the chair according to body type, needs and resources



# STRONGLY ENGINEERED AND FUNCTIONALLY DESIGNED EXCLUSIVELY FOR BARIATRIC USERS!

## SPECIFICATIONS


Maximum weight capacity is	270 /450 kg
Seat width	50 / 55 / 61 / 66 / 72 / 76 / 82 / 92 / 112 cm
Total width	Seat width + 18 cm
Total length without footrests	87 cm
Seat depth	41-56 cm
Seat height	36-51 cm
Backrest height	65-75 cm
Adjustable back angle	Yes
Adjustable height of armrests	Yes
Detachable armrests	Yes
Height adjustable footrests	Yes
Folding/detachable armrests	Yes
Weight	Approx. 30 kg
ISO - classification	ISO 12 22 03 - 08

## ORDERING

Model	Seat width cm	Seat depth cm	Order No
XXL-Rehab Eclipse incl. standard footrests	50	41-56	0120-050-000
XXL-Rehab Eclipse incl. standard footrests	55	41-56	0120-055-000
XXL-Rehab Eclipse incl. standard footrests	61	41-56	0120-061-000
XXL-Rehab Eclipse incl. standard footrests	66	41-56	0120-066-000
XXL-Rehab Eclipse incl. standard footrests	72	41-56	0120-072-000
XXL-Rehab Eclipse incl. standard footrests	82	41-56	0120-082-000
XXL-Rehab Eclipse incl. standard footrests	92	41-56	0120-092-000
XXL-Rehab Eclipse incl. standard footrests	102	41-56	0120-102-000
XXL-Rehab Eclipse incl. standard footrests	112	41-56	0120-112-000
<b>Accessories</b>			
Holder for oxygen bottle			0120-999-021
IV holder			0120-999-020
Headrest, adjustable			0121-999-010
Table (for seat width 55-72 cm)			0100-099-001
Disc brakes			0120-999-022B
Grow kit 5-16 cm			0120-999-030
Reinforcement of Eclipse to the capacity of 450 kg			0120-999-010
Reclining backrest			0120-999-032
Back extension			0120-999-033
Armrest desklength 12"-16" right			0120-999-078
Armrest desklength left			0120-999-077
Footrest standard swing-away right			0120-999-008
Footrest standard swing-away left			0120-999-009
Footrest elevating right			0120-999-006
Footrest elevating left			0120-999-007
Footplate "One-Piece" to seat width 61 cm			0120-061-010
Footplate "One-Piece" to seat width 66 cm			0120-066-010
Footplate "One-Piece" to seat width 72 cm			0120-072-010
Footplate "One-Piece" to seat width 82 cm			0120-082-010
Center mount. front rigging			0120-999-034
Transit tie down kit			0120-999-065



30° hydraulic back

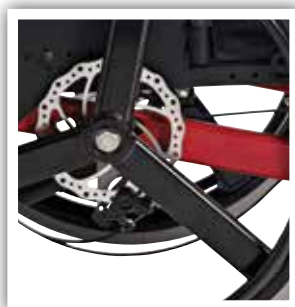

Disc brakes

# XXL REHAB<sup>®</sup>

## MANUEL WHEELCHAIR - THE ECLIPSE TILT

The tilt mechanism allows up to 30 degrees tilt for optimal comfort and positioning for individuals needing more support. The tilt uplift force is adjustable allowing the chair to be easily tilted regardless of user weight.

The steel frame gives the XXL-Rehab Eclipse Tilt Wheelchair maximum strength and rigidity, and therefore also makes it a long lasting wheelchair.

The frame is designed so most of the user weight is carried on the rear wheels. The wheels are especially made for this wheelchair; they are extra hard and solid making propulsion and steering easy and with very little loss of energy!

The seat pan is solid. Various medical conditions demand a solid seatpan (fractures, hemiplegia, lateral shifts etc). Giving a bariatric user a solid seat pan facilitates a good posture and activeness in the chair.

The XXL-Rehab Eclipse Tilt Wheelchair has adjustable seat depth, seat height, seat to floor angle and seat to back angle. All of this makes it easy to adjust to any user!

The back is constructed so users with a protruding behind can get good support to their lumbar spine, a huge problem rarely met by any other chair!

Front wheels are placed in front of the seat pan to ensure an anterior anti-tip, which is very important taking into consideration that most bariatric users have a lot of weight placed in front of them. The anterior anti-tip makes it possible for the user to get in and out of the wheelchair without the risk of tipping.

Different types of front rigging complete the adjustment possibilities. The chair can be used with 3 different kinds of cushions.

- 30° tilt
- Strong construction




# STRONGLY ENGINEERED AND FUNCTIONALLY DESIGNED EXCLUSIVELY FOR BARIATRIC USERS!

## SPECIFICATIONS

Maximum weight capacity is	270 kg
Seat width	50 / 55 / 61 / 66 / 72 / 82 cm
Total width	Seat width + 25 cm
Total length without footrests	87 cm
Seat depth	41-56 cm
Seat height	36-51 cm
Backrest height	65-75 cm
Adjustable back angle	Yes
Adjustable height of armrests	Yes
Detachable armrests	Yes
Height adjustable footrests	Yes
Folding/detachable armrests	Yes
Weight	Approx. 32 kg
ISO - classification	ISO 12 22 03 - 08

## ORDERING

Model	Seat width cm	Seat depth cm	Order No
XXL-Rehab Eclipse Tilt incl. standard footrests	50	41-56	0121-050-000
XXL-Rehab Eclipse Tilt incl. standard footrests	55	41-56	0121-055-000
XXL-Rehab Eclipse Tilt incl. standard footrests	61	41-56	0121-061-000
XXL-Rehab Eclipse Tilt incl. standard footrests	66	41-56	0121-066-000
XXL-Rehab Eclipse Tilt incl. standard footrests	72	41-56	0121-072-000
XXL-Rehab Eclipse Tilt incl. standard footrests	82	41-56	0121-082-000
Accessories			
Holder for oxygen bottle			0120-999-021
IV holder			0120-999-020
Headrest, adjustable			0121-999-010
Table (for seat width 55-72 cm)			0100-099-001
Back extension			0120-999-033
Armrest desklength 12"-16" right			0120-999-078
Armrest desklength left			0120-999-077
Footrest standard swing-away right			0120-999-008
Footrest standard swing-away left			0120-999-009
Footrest elevating right			0120-999-006
Footrest elevating left			0120-999-007
Footplate "One-Piece" to seat width 61 cm			0120-061-010
Footplate "One-Piece" to seat width 66 cm			0120-066-010
Footplate "One-Piece" to seat width 72 cm			0120-072-010
Footplate "One-Piece" to seat width 82 cm			0120-082-010
Transit tie down kit			0120-999-065

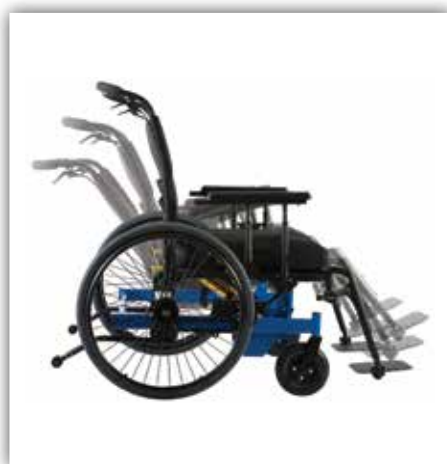

Tilt in space



Gas strut



# XXL REHAB<sup>®</sup>

## HD ELECTRIC WHEELCHAIR

The XXL-Rehab HD electric wheelchair is the obvious choice for bariatric users. It has a heavy-duty construction with upgraded in-line dual motors and a reinforced frame for maximum durability. The wheelchair also offers solid springs on the front anti-tip. The electric wheelchair is simple and easy to use and many adjustments are possible to ensure a high degree of comfort. The wheelchair is equipped with the seating system "Synergy" which, among other things, can grow in both width and depth.

The wheelchair has superb maneuverability with a turning radius of only 131 cm. The patented mid-wheel drive design together with 16" drive wheels and 9" caster wheels provide unique stability for the user.

The wheelchair is both very reliable and safe to use. The operation of the wheelchair is quiet with a top speed of 8 km/h and it has Intelligent Braking which is electronic, regenerative disc brakes.

- Adjustable in seat width and depth
- Superb maneuverability
- Turns on minimum space



**STRONGLY ENGINEERED AND FUNCTIONALLY DESIGNED  
EXCLUSIVELY FOR BARIATRIC USERS!**

**SPECIFICATIONS**

Maximum weight capacity is	295 kg
Seat width	61-71 cm
Total width	78 cm
Total length with footrests	130 cm
Seat depth	30-55 cm
Tilt	5°
Seat height without cushion	46-51 cm
Height seat to armrest	Adjustable
Weight with seat and batteries	Approx. 165 kg
Ground clearance	10 cm
Height adjustable armrests	Yes
Adjustable footrests	36-43.5 cm
Folding footrests	Yes
Removable footrests	Yes
Height adjustable footrests	Yes
Head rest	Yes
Speed, max.	8 km/h
Range per charge	Approx. 24 km
Turning radius	131 cm
Drive wheels	16"
Front casters	9"
Rear casters/anti-tip wheels	8"
Batteries	Two 12 volt, deep cycle, Group 24
Battery weight	24 kg x 2
Battery charger	8 Amp, off-board
ISO - classification	ISO 12 23 06 - 08

**ORDERING**

Model	Seat width cm	Seat depth cm	Order No
XXL-Rehab HD electric wheelchair	61-71	30-55	0500-066-000
<b>Accessories</b>			
Headrest, adjustable			0500-099-010
Footrest, angle adjustable right			0500-099-020
Footrest, angle adjustable left			0500-099-021
Lap belt, adjustable 228 cm			0500-099-030
Dual cane holder			0500-099-040
Rear basket			0500-099-050
Swing-away joystick mount. right			0500-099-060
Swing-away joystick mount. left			0500-099-061




Rear basket



Dual cane holder



Swing-away joystick mount

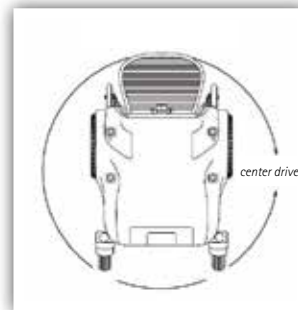

Turning radius 131 cm

# XXL REHAB®

## WHEELCHAIR CUSHIONS

### XXL-Rehab Comfort cushion

This cushion is developed exclusively for bariatric users who need prophylaxis that last over a longer periode. The cushion is made of viscoelastic foam on the top and springs cut into the underbase of foam.

The patented system of springs distribute the pressure to a larger surface and assure pressure relief.

The cushion is 12 cm high and has a 2 way stretch vapour permeable incontinence cover with a non slip base.

- Viscoelastic top
- 2 way stretch incontinence cover

### XXL-Rehab Anatomical cushion

This wheelchair cushion is designed exclusively for bariatric users at risk of developing pressure sores.

It is made of a molded base 11 cm firm foam and with a 3 cm Viscoelastic foam on the top.

The molded base provides stability and seat support and provides unique pressure relief on the Tuber Ischii.

The cushion is 14 cm high and has a 2 way stretch vapour permeable incontinence and flame resistant cover.

- Anatomically shaped
- 2 way stretch incontinence cover

### XXL-Rehab Plus cushion

This wheelchair cushion is designed exclusively for bariatric users at risk of developing pressure sores.

It is made of a molded base and has a sealed silicone bag in the middle. The molded base provides stability and seat support and the silicone bag provides unique pressure relief on the Tuber Ischii.

The cushion is 10 cm high and has a 2 way stretch vapour permeable incontinence and flame resistant cover with a non slip base.

- Unique pressure relief
- 2 way stretch incontinence cover





**PRESSURE RELIEF DESIGNED  
EXCLUSIVELY FOR BARIATRIC USERS!**

## XXL-Rehab Comfort cushion

### SPECIFICATIONS

Maximum weight capacity is	325 kg
Weight	Approx. 2 kg
Basis cushion	HD PUR foam / Viscoelastic PUR foam
Cover	PU
Incontinence cover	Yes
Washable cover	Yes
Flame retardant cover	Yes

### ORDERING

Model	Seat width cm	Seat depth cm	Height cm	Maximum weight capacity	Order No
XXL-Rehab Comfort cushion	55	45	12	325	0230-055-045
XXL-Rehab Comfort cushion	61	45	12	325	0230-061-045
XXL-Rehab Comfort cushion	66	45	12	325	0230-066-045
XXL-Rehab Comfort cushion	72	45	12	325	0230-072-045
XXL-Rehab Comfort cushion	82	45	12	325	0230-082-045

## XXL-Rehab Anatomical cushion

### SPECIFICATIONS

Maximum weight capacity is	325 kg
Weight	Approx. 3 kg
Basis cushion	HD PUR foam / Viscoelastic PUR foam
Cover	PU
Incontinence cover	Yes
Washable cover	Yes
Flame retardant cover	Yes

### ORDERING

Model	Seat width cm	Seat depth cm	Height cm	Maximum weight capacity	Order No
XXL-Rehab Anatomical cushion	56	50	14	325	0233-056-050
XXL-Rehab Anatomical cushion	61	50	14	325	0233-061-050
XXL-Rehab Anatomical cushion	66	50	14	325	0233-066-050
XXL-Rehab Anatomical cushion	71	50	14	325	0233-071-050
XXL-Rehab Anatomical cushion	81	50	14	325	0233-081-050
XXL-Rehab Anatomical cushion	91	50	14	325	0233-091-050
XXL-Rehab Anatomical cushion	96	50	14	325	0233-096-050
XXL-Rehab Anatomical cushion	100	50	14	325	0233-100-050

## XXL-Rehab Plus cushion

### SPECIFICATIONS

Maximum weight capacity is	152 / 175 / 195 kg
Weight	Approx. 2.5 kg
Basis cushion	Molded foam
Gel	100% silicone
Cover	Polyamide / polyurethane
Incontinence cover	Yes
Washable cover	Yes
Flame retardant cover	Yes

### ORDERING

Model	Seat width cm	Seat depth cm	Height cm	Maximum weight capacity	Order No
XXL-Rehab Plus cushion	50	43-46-50	10	152	0232-050-000
XXL-Rehab Plus cushion	55	46-50-55	10	175	0232-055-000
XXL-Rehab Plus cushion	61	46-50-55	10	195	0232-061-000

# XXL REHAB<sup>®</sup>

## STAND TALL

The XXL-Rehab Stand Tall is a multi-functional aid; It can be used for walking, mobilising from seated to standing, it is safe to use, gives independence and has an IV option.

The XXL-Rehab Stand Tall has 2 built-in handgrips on the side which allow the user to stand up in small steps from the bed or chair being in full control of the situation. This minimises the risk of falling and therefore also minimises the risk of injury to caregiver or patient himself. Once upright the user can walk around.



The underarm-padding is adjustable in width and angle giving optimal support to the user. This function is supported in full length of the bar.

In addition, it has many handgrip-combinations and easy adjustment, everything to make the XXL-Rehab Stand Tall easy to maneuver.

The brakes are mounted onto the bar, useful in stops of course, but also in speed control and turning: the XXL-Rehab Stand Tall is a safe zone. The brakes can be locked, i.e. when getting up/sitting down.

Using 32 mm tubing ensures an almost forever lasting frame, but also gives a much more simple look and it can be cleaned in few seconds.

IV holder and holder for urine-bag are optional.

The XXL-Rehab Stand Tall is mainly designed for hospital use, but can be used at home as well.

- Height adjustable
- Maximum weight capacity up to 325 kg



Product video  
Scan the code and see  
the product in use

**STRONGLY ENGINEERED AND FUNCTIONALLY DESIGNED  
EXCLUSIVELY FOR BARIATRIC USERS!**

**SPECIFICATIONS**

Maximum weight capacity is	325 kg
Width	76 cm
Adjustable height	100-130 cm
Internal width	71 cm
External width between armrests	55 cm
Depth	72 cm
Front wheel	100 mm
Rear wheel	125 mm
Frame	Powder-lacquered chromium molybdenum steel
ISO - classification	ISO 12 06 12 - 06

**ORDERING**

Model	Width cm	Length cm	Order No
XXL-Rehab Stand Tall	76	73	0180-076-000
<b>Accessories</b>			
IV holder for Stand Tall			0180-099-008



# XXL REHAB®

## ROLLATOR KING

**STRONGLY ENGINEERED AND FUNCTIONALLY DESIGNED EXCLUSIVELY FOR BARIATRIC USERS!**

With a weight capacity of 325 kg the XXL-Rehab Rollator King is one of the strongest walkers.

It is both functional and has a light and discrete design. It fits through doorways and folds easily for transport and storage in cars.

It includes a large seat, extra strong tubes and 8" massive tires that provide extra security, stability and comfort.

The XXL-Rehab Rollator King has a large support surface. When the user sits down heavily, the back arch prevents that the weight is placed too far – beyond the support surface. In this way, the risk of accidents is prevented significantly.

The critical point for rollators for bariatric users are the brakes. The XXL-Rehab Rollator King has "feather touch" hand brakes that provide superior control and ensure the walker is easy to maneuver and to hold the weight.

The rollator comes with a basket and the handles are fitted with reflectors.

- **Extremely strong and stable lightweight**
- **Easy to fold and maneuver**
- **"Feather touch" hand brakes**
- **Fits through doorways and into cars**
- **The new design with iron forks and wide tires gives a smoother drive and safe handling**



### SPECIFICATIONS

Maximum weight capacity is	325 kg
Seat dimension	35.5 x 50.8 cm
Width between handles	61 cm
Total width	71 cm
Seat height	50 cm
Adjustable height	77.5-99 cm (2.5 cm interval)
Weight	15.7 kg
ISO - classification	ISO 12 06 06 - 11

### ORDERING

Model	Order No
XXL-Rehab Rollator King	0170-050-000

# XXL REHAB®

## ROLLATOR SUPPORT

STRONGLY ENGINEERED AND FUNCTIONALLY DESIGNED  
EXCLUSIVELY FOR BARIATRIC USERS!

With a weight capacity of 200 kg the XXL-Rehab Rollator Support is one of the strongest walkers.

It is both functional and has a light and discrete design. It fits through doorways and folds easily for transport.

It includes a large seat, extra strong tubes and 8" massive tires that provide extra security, stability and comfort.

The XXL-Rehab Rollator Support has a large support surface. When the user sits down heavily, the back arch prevents that the weight is placed too far – beyond the support surface. In this way, the risk of accidents is prevented significantly.

The critical point for rollators for bariatric users are the brakes. The XXL-Rehab Rollator Support has "feather touch" hand brakes that provide superior control and ensure the walker is easy to maneuver and to hold the weight.

The rollator comes with a basket and the handles are fitted with reflectors.

- Extremely strong and stable lightweight
- Easy to fold and maneuver
- "Feather touch" hand brakes
- Fits through doorways
- The underarm support helps to walk in a more upright position and minimize stress on wrist



MAX 200 KG

### ORDERING

Model	Order No
XXL-Rehab Rollator Support	0171-050-000

### SPECIFICATIONS

Maximum weight capacity is	200 kg
Seat dimension	35.5 x 50.8 cm
Seat width	61 cm
Seat height	50 cm
Width between handles	61 cm
Total width	71 cm
Adjustable height	100-130 cm (2.5 cm interval)
Weight	17 kg
ISO - classification	ISO 12 06 06 - 11

## BARIATRIC WALKING FRAME

Especially after falling incidences, fractures etc one of the key factors of getting bariatric users mobilised is to provide them with a walking aid they trust.

The XXL-Rehab Bariatric Walking Frame is foldable and available in 2 widths, with ferrules or with 2 massive wheels in front. With 2 wheels in front the walking frame is easy to maneuver and roll across the floor. The 2 rear legs ensure that walking frame does not suddenly roll away if the user for example loses balance.

The walking frame is lightweight but solid and offers a high degree of stability. It has padded handles and the height can easily be adjusted from 76-101 cm without tools. The XXL-Rehab Bariatric Walking Frame is specially designed for users up to 270 kg.

Folds easily for transport and storage.

- **Designed exclusively for bariatric users**
- **Foldable**
- **Height adjustable**
- **Lightweight frame**
- **Maximum weight capacity up to 270 kg**



**STRONGLY ENGINEERED AND FUNCTIONALLY DESIGNED  
EXCLUSIVELY FOR BARIATRIC USERS!**

**SPECIFICATIONS**

Maximum weight capacity is	270 kg
Total length	50 cm
Total width	74 / 84 cm
Inside width	54 / 64 cm
Adjustable height in 2.5 cm interval	76-101 cm
Weight	3 kg
Foldable	Yes
Frame	Powder coated light metal
Grip	PU foam
Wheels	Massive
ISO – classification	ISO 12 06 03 - 06

**ORDERING**

Model	Width cm	Length cm	Order No
XXL-Rehab Bariatric Walking Frame without wheels	74	50	0130-071-000
XXL-Rehab Bariatric Walking Frame without wheels	84	50	0130-081-000
XXL-Rehab Bariatric Walking Frame with wheels	74	50	0130-074-000
XXL-Rehab Bariatric Walking Frame with wheels	84	50	0130-084-000



*XXL-Rehab Bariatric Walking Frame folded*

### XXL-Rehab Walking cane

Extra reinforced cane. Exclusively designed for bariatric users weighing up to 270 kg.

The curved neck places the users weight directly over the shaft for stability and safety. The cane has a vinyl handle and double push-button for adjustment of height.

### XXL-Rehab 4-Point cane

Extra reinforced cane. Exclusively designed for bariatric users.


The cane has an adjustable handle and large double push-buttons for adjustment of height.

The 4-point base increases the stability.

### XXL-Rehab Crutch

Exclusively designed for bariatric users weighing up to 325 kg.

New carbon design.

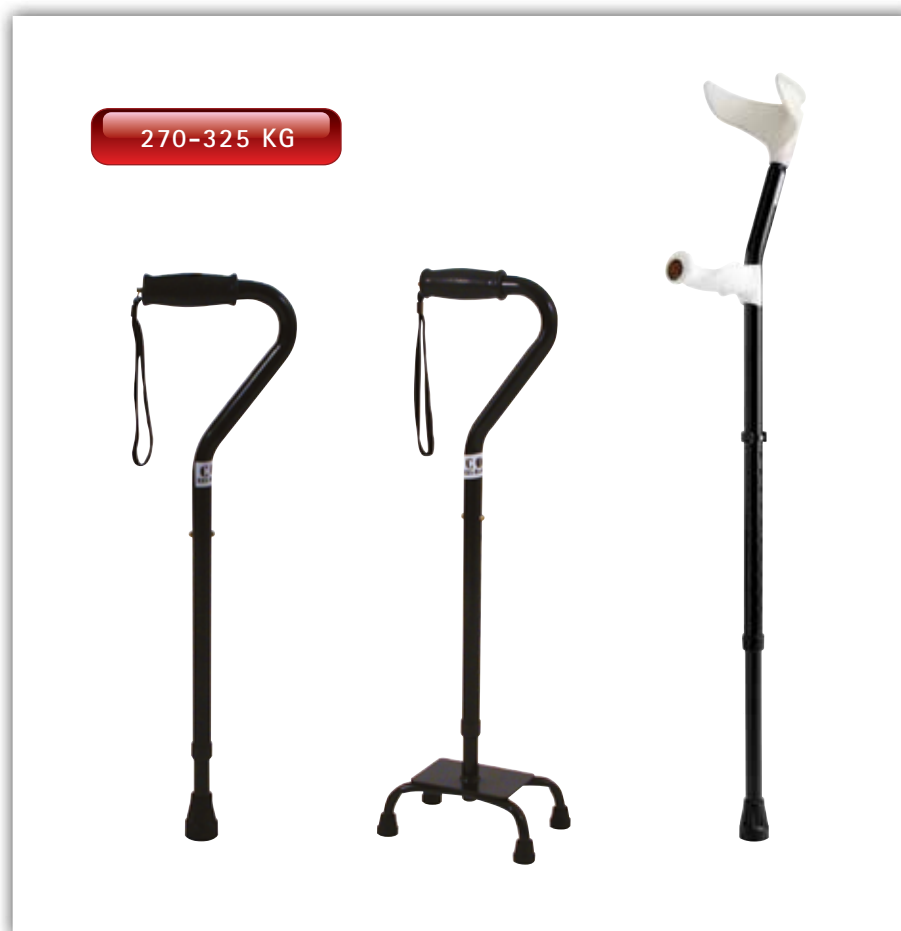



**STRONGLY ENGINEERED AND FUNCTIONALLY DESIGNED  
EXCLUSIVELY FOR BARIATRIC USERS!**

**SPECIFICATIONS  
XXL-REHAB WALKING CANE**

Maximum weight capacity is	270 kg
Height adjustable	71-93 cm
Weight	0.9 kg
Material	Anodized steel
Grip	Vinyl
ISO - classification	ISO 12 03 03 - 05

**SPECIFICATIONS  
XXL-REHAB 4-POINT CANE**


Maximum weight capacity is	270 kg
Height adjustable	71-93 cm
Weight	1.5 kg
Base measurement	15 x 22 cm
Reflex	Yes
Material	Anodized steel
Grip	Vinyl
ISO - classification	ISO 12 03 16 - 03

**SPECIFICATIONS  
XXL-REHAB CRUTCH**

Maximum weight capacity is	325 kg
Height adjustable	68-110 cm
Weight	1.5 kg
Underarm support	15 cm
Grip	14 cm
Anatomical grip	Yes
Reflex	Yes
Material	Anodized steel
Grip	PU/Vinyl
ISO - classification	ISO 12 03 06 - 04

**ORDERING**

Model	Order No
XXL-Rehab Walking cane	0200-000-000
XXL-Rehab 4-Point cane	0201-000-000
XXL-Rehab Crutch with universal grip	0205-000-000
XXL-Rehab Crutch with anatomical grip right	0205-000-000R
XXL-Rehab Crutch with anatomical grip left	0205-000-000L

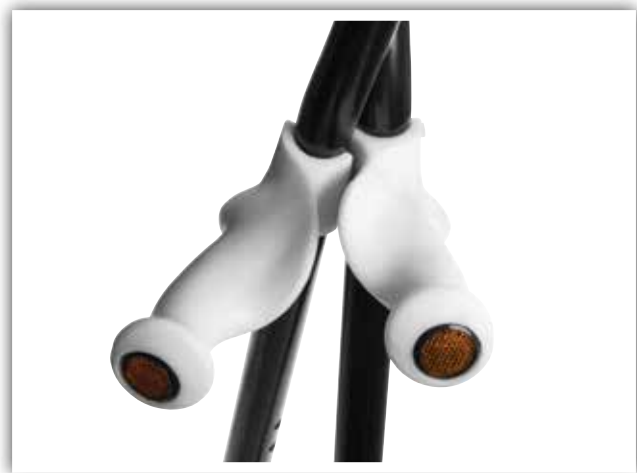

*XXL-Rehab Crutch with anatomical grip*

# XXL REHAB<sup>®</sup>

## NURSING BED

The XXL-Rehab Nursing Bed is designed for the bariatric user who needs an extra wide bed. It is available in width 120 cm. The XXL-Rehab Nursing Bed is a modern care bed with a smart design that fits both nursing homes and private homes. The bed meets both the needs of the user and the caregiver.

### Robust and stable

Is very stable and therefore very suitable for those who can transfer themselves. A stable frame with 8 locking wheels ensures that the bed does not tilt or shake dangerously if the user sits down heavily. The bed is reinforced and has a unique sturdy scissors construction. It comes with both headboards and bed rails.

### Electric operation

The lying surface of the bed is divided into four with up / down feature for both the headboard and foot end. The up / down feature is operated via hand control. The hand control is clearly illustrated with the function of each button and can be placed at either side of the bed.

### Good working positions

This bed is also very practical for the caregiver. The height adjustable bed surface ensures good working positions for the caregiver.

- Maximum weight capacity up to 300 kg
- Smart design that fits into private homes and nursing homes



**STRONGLY ENGINEERED AND FUNCTIONALLY DESIGNED  
EXCLUSIVELY FOR BARIATRIC USERS!**

**SPECIFICATIONS**

Maximum weight capacity is	300 kg
Total length	230 cm
Total width	130 cm
Mattress size length	220 cm
Mattress size width	120 cm
Lying surface height	40-80 cm
Weight	200 kg
Frame	Powder coated steel
Bed frame	Wood veneer
Wheels	8, 100 mm, lockable
Ingress for water	66 IP
ISO - classification	ISO 18 12 10 - 30

**ORDERING**

Model	Length cm	Width cm	Order No
XXL-Rehab Nursing Bed	120	200	2312-120-200
<b>Accessories</b>			
Lifting pole			2312-999-010



*Hand control*



# XXL REHAB®

## 400 BARIATRIC HOSPITAL BED

The XXL-Rehab 400 bariatric hospital bed is a modern nursing bed suitable for bariatric users who weigh up to 400 kg. This bed is highly appropriate for recycling due to its adjustability.

The bed accommodates both the needs of the user and caregiver. The bed is manually adjustable both in width (90 /100 or 120 cm) and in length (200 or 220 cm), making it easy to use for multiple users with different needs, both in hospitals and in home care.

A stable frame and a sturdy construction ensure that the bed provides great stability for the user, also in connection with transfer. The bed also has a central braking system and directional control of the wheels. Light bed rails are available for the bed. They are removable without tools. The lying surface of the bed is divided into four with up / down feature for both the headboard and foot end.

The XXL-Rehab 400 hospital bed has separate controls for the caregiver, including Trendelenburg and anti-Trendelenburg. Operation is easy to use and the caregiver can for example adjust the bed by standing at the foot end. The caregiver also has the option to lock individual power functions; this can help prevent misalignments of the user in the bed.

The bed has a light and elegant design with rounded corners, making it very easy to clean.

- Bed rails are removable without tools
- Many adjustments available for both user and caregiver
- Very easy to clean



*Width adjustable in both sides*



## STRONGLY ENGINEERED AND FUNCTIONALLY DESIGNED EXCLUSIVELY FOR BARIATRIC USERS!

### SPECIFICATIONS

Maximum weight capacity is	400 kg
Total length	231-251 cm
Total width	100-128 cm
Mattress size length	200 / 220 cm
Mattress size width	90 / 100 / 120 cm
Manually adjustable in width	90 / 100 / 120 cm
Manually adjustable in length	200 / 220 cm
Lying surface height	39-82 cm
Minimum clear distance from the floor	15 cm
Weight	175 kg
Separate control for caregiver	E.g. Trendelenburg and Anti-Trendelenburg position
Direction control on the wheels	Yes
Central brake	Yes
Magnetic lock on the user hand control	Yes
Lock of individual electric functions	Yes
Removable lifting pole	Yes
Removable foot gable	Yes
Removable headboard	Yes
Ingress for water	66 IP
Wheel diameter	145 mm
ISO - classification	ISO 18 12 10 - 30

### ORDERING

Model	Order No
XXL-Rehab 400 bariatric hospital bed	2314-120-220
Accessories	
Bed rails	2314-999-008
Lifting pole	2314-999-010
IV pole	2314-999-012



Control panel for caregiver



Hand control



# XXL REHAB®

## COMFORT FOAM MATTRESS

PRESSURE RELIEF DESIGNED EXCLUSIVELY FOR BARIATRIC USERS!

The XXL-Rehab Comfort foam mattress is a standard mattress which works perfectly with nursing beds. The mattress secures a soft but still effective distribution of the load combined with a unique stability that helps the user to obtain the ability to reposition himself/herself.

The mattress is mainly intended for bariatric users with a low risk of developing pressure sores.

The bottom part of the mattress consists of a 15 cm layer of heavy duty PU foam cut into springs. This feature secures a lower "push back" effect on the user and at the same time ensures an air flow within the mattress, thereby minimizing the accumulation of heat.

On top of the 15 cm PU foam lies a 3 cm layer of viscoelastic foam. This layer ensures maximum adaption to the surface of the user and thereby also a greater weight distribution and pressure relief up to grade 2.

The XXL-Rehab Comfort foam mattress has a 2 way elastic incontinence cover. The cover is antibacterial, antiallergic and flame retardant.. It can be machinewashed.

The XXL-Rehab Comfort Foam mattress can be turned horizontally in the bed to prolong the life of the mattress.

- Fits perfectly in nursing beds
- Pressure relief mattress that also ensures the mobility of the user



Patented design

### SPECIFICATIONS

Model 1      Model 2

Maximum weight capacity is	325 kg	
Length	200 cm	220 cm
Width	100 cm	120 cm
Height	18 cm	
Weight	12 kg	13 kg
Mattress	Viscoelastic PUR foam / Heavy Duty PU foam	
Cover	100% polyurethan	
ISO - classification	ISO 18 12 18 -25	

### ORDERING

Model	Length cm	Width cm	Height cm	Order No
XXL-Rehab Comfort foam mattress	200	100	18	0240-100-200
XXL-Rehab Comfort foam mattress long	220	100	18	0240-100-220
XXL-Rehab Comfort foam mattress	200	120	18	0240-120-200
XXL-Rehab Comfort foam mattress long	220	120	18	0240-120-220

# XXL REHAB®

## LOW AIR LOSS MATTRESS

PRESSURE RELIEF DESIGNED EXCLUSIVELY FOR BARIATRIC USERS!

Bariatric users often have difficulty in repositioning themselves in the bed, or only tolerate a certain lying position because of physical restrictions. If you add internal factors such as capillary flow, oxygen tension, sweating and additional illnesses it seems obvious that pressure relieving is of great importance for the bariatric users.

The XXL-Rehab Low Air Loss mattress is for bariatric users with a long history of pressure sores, with high risk of developing new pressure sores or with existing pressure sores up to grade 4.

Very immobile and bedridden patients benefit a lot from the features of the XXL-Rehab Low Air Loss mattress. The constant flow of air helps reduce the maceration of the skin due to moisture/sweat. The blower makes it easy to max inflate the mattress within seconds, making it easy to help the patients with hygiene, help turn the patient or provide stability for the patient so he or she can mobilise himself/herself.

The mattress is extra wide and the cells are reinforced and PU coated. It has a maximum weight capacity of 350 kg, and comes with a 2 way stretch incontinence cover with antibacterial coating.

The pump has an easily understandable control panel with different functions such as "firm" and "sitting function". Activating "firm" makes the mattress evenly hard for 20 minutes. After that it returns to the previously pressed mode.

- Static head section and safety base
- CPR control



MAX 350 KG



### SPECIFICATIONS

Maximum weight capacity is	350 kg
Length	200 cm
Width	90 / 100 / 120 cm
Height	26 cm
Weight	12 / 16 kg
Weight of pump	4.6 kg
Pump measurement	36 x 30 x 17 cm
Cover	100% nylon with PU on the back
Cells	100% nylon with TPU lamination
Pump	ABS flame retardant UL
ISO - classification	ISO 18 12 18 - 28

### ORDERING

Model	Length cm	Width cm	Height cm	Order No
XXL-Rehab Low Air Loss mattress with pump	200	90	26	0242-090-200
XXL-Rehab Low Air Loss mattress with pump	200	100	26	0242-100-200
XXL-Rehab Low Air Loss mattress with pump	200	120	26	0242-120-200

# XXL REHAB®

## ALTERNATING MATTRESS

PRESSURE RELIEF DESIGNED EXCLUSIVELY FOR BARIATRIC USERS!

Bariatric users often have difficulty in repositioning themselves in the bed, or only tolerate a certain lying position because of physical restrictions. If you add internal factors such as capillary flow, oxygen tension, sweating and additional illnesses it seems obvious that pressure relieving is of great importance for the bariatric users.

The XXL-Rehab Alternating mattress is for users with a history of pressure sores, with the risk of developing new pressure sores or with existing pressure sores up to grade 3.

The XXL-Rehab Alternating mattress is controlled by a microprocessor run pump which works in an A-B cycle. The cycle time is adjustable from 10 – 20 minutes. In this way, the mattress creates movement under the user. Combined with a cell height of 21 cm it is possible to both relieve and distribute the load.

The mattress holds different functions such as "firm". Activating "firm" makes the mattress evenly hard for 20 minutes. After 20 minutes it returns to alternating. This function is of great importance for the caregiver, because it makes it much easier to either facilitate transfer, training or help in nursing situations.

The pump is almost silent due to the microprocessor run pump. It also holds an alarm function for power supply, malfunction and in case of air loss – the alarm is both visual and audible.



Pump with clearly understandable display



Easy CPR function

MAX 270 KG



### SPECIFICATIONS

Maximum weight capacity is	270 kg
Length	200 cm
Width	100 / 120 cm
Height	27 cm
Weight	14 / 18 kg
Weight of pump	3.8 kg
Pump measurement	35 x 13 x 24 cm
Cover	PU
Cells	Nylon / PU
ISO - classification	ISO 18 12 18 - 28

### ORDERING

Model	Length cm	Width cm	Height cm	Order No
XXL-Rehab Alternating mattress with pump	200	100	27	0244-100-200
XXL-Rehab Alternating mattress with pump	200	120	27	0244-120-200





# XXL REHAB<sup>®</sup>

## FLOOR RAISER

The bed is often too high for feet to reach the floor and this complicates getting in and out for the user. The XXL-Rehab Floor Raiser is a platform to put beside the bed to make this much easier.

The floor raiser may be the gap between dependent and independent living with the possibility of getting in and out of the bed.

For the caregiver the XXL-Rehab Floor Raiser also provides new possibilities for mobilising the patient with less danger of gliding out of the bed and risk of falling.



STRONGLY ENGINEERED AND FUNCTIONALLY DESIGNED  
EXCLUSIVELY FOR BARIATRIC USERS!

SPECIFICATIONS

Width	80 cm
Length	100 cm
Height adjustment	15 / 17.5 / 20 cm
Weight	18 kg

ORDERING

Model	Order No
XXL-Rehab Floor Raiser	2320-010-000



# XXL REHAB<sup>®</sup>

## PATIENT/DINNER CHAIR

The armrests are constructed so they go beyond the seat. This makes it easy both to get a good grip when a person wants to sit down but also to get up again. We have designed the armrests to bend out like a butterfly, so the body doesn't get squeezed, getting in and out the chair. The butterfly armrests of the XXL-Rehab Patient/Dinner chair also provide much better functionality for the user, compared to other chairs with straight and short armrests where it is almost impossible for a bariatric user to get a good grip.

The height of the chair is adjustable from 44 to 59 cm. It is very important to adjust the chair to the right height to have success getting in and out of the chair. If the chair is too high for the user, the feet will lift off the ground, making it hard to get seated in the chair. If the chair is too low, the user will have trouble getting up.

It is often a problem for bariatric users to be seated comfortably. Sometimes the seat depth is too short, sometimes too deep and with no back support. We have designed the seat to be individually adjusted to provide a comfortable seating. The backrest can be moved +/- 6 cm and the bended tubing assures that the right comfort can be found.

It is made easy to both handle and move the chair from place to place. In the construction of the chair we have only used high-tech lightweight tubing, to provide the highest strength at the lowest weight.



**STRONGLY ENGINEERED AND FUNCTIONALLY DESIGNED  
EXCLUSIVELY FOR BARIATRIC USERS!**

**SPECIFICATIONS**

Model 1    Model 2

Maximum weight capacity is	325 kg	
Seat width	61 cm	71 cm
Total length	80-101 cm	
Seat depth	40-52 cm	
Seat height	44-59 cm	
Backrest height	37 cm	
Distance from floor to seat underside	33-48 cm	
Total width	69 cm	79 cm
Total height	79-94 cm	
Total weight	16.6 kg	17.6 kg
Total outer width on armrests	74 cm	84 cm
Seat / back / armrests	Polyurethan	
Frame	Powder-lacquered chromium molybdenum steel	
ISO - classification	ISO 09 33 03	

**ORDERING**

Model	Seat width cm	Seat depth cm	Seat height cm	Order No
XXL-Rehab Patient/Dinner Chair	61	40-52	44-59	0160-061-000
XXL-Rehab Patient/Dinner Chair	71	40-52	44-59	0160-071-000



# XXL REHAB<sup>®</sup>

## OFFICE CHAIR

This office chair is comfortable and provides good working positions at any place. The seat measures 62 x 47 cm and is well padded, providing good pressure relief and a comfortable sitting position for a long time.

The chair can be adjusted both in width (62-74 cm) and in depth (42-52 cm) by moving the armrests and the back. The user can transfer vertically due to the electric seat hoist and the smooth-running wheels, making it easier to live an active life at home.

The chair has a strong safety construction, durable wheels with heavy duty bearings easy to drive and a weight capacity of 325 kg. The chair has a hand operated brake.

XXL-Rehab Office Chair has electric height adjustment with one cylinder which makes the hoist extra strong. The hoist has a lift span of 20 cm which makes the chair easy to adjust to the correct working height. The electric hoist is easy to operate pushing the button located under the armrest.

The backrest is reinforced and mounted on a tube which can be adjusted in angle and depth. The armrests can be folded down and have continuously adjustable conical track, ensuring that they never detach.

The office chair comes with strong upholstery that is stain resistant and fire retardant.

- Adjustable both in width and in depth
- Maximum weight capacity up to 325 kg



**STRONGLY ENGINEERED AND FUNCTIONALLY DESIGNED  
EXCLUSIVELY FOR BARIATRIC USERS!**

**SPECIFICATIONS**

Maximum weight capacity is	325 kg
Adjustable seat width	62-74 cm
Total width	77-90 cm
Adjustable height	55-75 cm
Adjustable depth	42-52 cm
Back upholstery	61 x 38 cm
Adjustable backrest height	43-56 cm
Seat upholstery	62 x 47 cm
Weight	Approx. 50 kg
ISO - classification	ISO 18 09 03 - 12

**ORDERING**

Model	Seat width cm	Seat depth cm	Seat height cm	Order No
XXL-Rehab Office Chair	62-74	42-52	55-75	0190-062-145



# XXL REHAB<sup>®</sup>

## PATIENT LIFT AND SLINGS

The XXL-Rehab Patient Lift is designed to make hoisting a simple and functional task. The lift is both easy to maneuver, uncomplicated to operate and provides security for both user and operator.


The lift is supplied with an electrical V-shaped leg-opening mechanism with a maximum loading capacity of 300 kg.

A built-in weighing scale is also available.

The XXL-Rehab Lifting sling provides comfort and safety for transfer in lying and seated positions.

Our slings can be used on any mobile or ceiling-mounted lift that utilises a hanger with hooks for sling strap attachment.

- Low battery alarm
- Lifting from floor

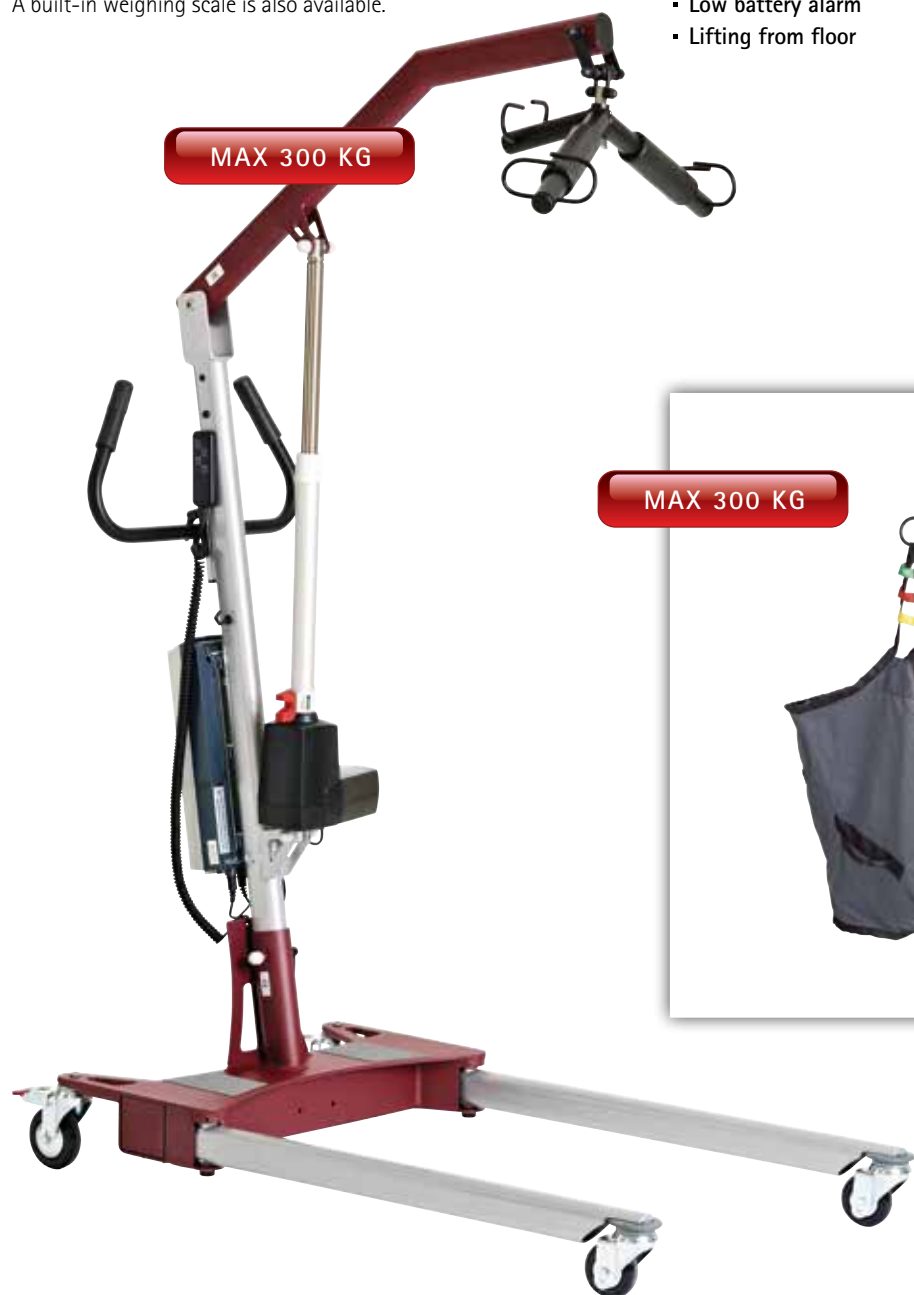



**STRONGLY ENGINEERED AND FUNCTIONALLY DESIGNED  
EXCLUSIVELY FOR BARIATRIC USERS!**

**SPECIFICATIONS**

Maximum weight capacity is	300 kg
Lifting height	46-170 cm
Lifting interval	124 cm
Total length	117 cm
Total width	65-126 cm
Turning radius	132 cm
Weight	52 kg
ISO - classification	ISO 12 36 03 - 04

**ORDERING**

Model	Order No
XXL-Rehab Lift	0250-000-000
<b>Accessories</b>	
Universal sling with head support	0250-099-001
Universal shower sling with head support	0250-099-002
Universal half sling	0250-099-011
Universal half bath sling	0250-099-012
Toilet sling	0250-099-014
Built-in electronic weighing scale. Max. 400 kg	0250-099-020



*Universal sling with head support*



*Universal shower sling with head support*



*Universal half sling*



*Universal half bath sling*



*Toilet sling*



*Built-in electronic weighing scale  
Max. 400 kg*

## MULTIFUNCTIONAL SCALE

- MODULAR WHEELCHAIR SCALE OR BARIATRIC SCALE

The XXL-Rehab Multifunctional Scale can be built in modules according to needs and can be used to weigh wheelchair users, users with walkers or for bariatric users up to 300 kg. It is particularly suitable for hospitals, nursing homes and institutions.

### Construction

The XXL-Rehab Multifunctional Scale consists of an electronic weighing platform with two smooth running transport wheels and transport handles and an easily readable LCD display. The weighing platform can be combined with ramp, safety edges and 2 kinds of heavy railings. The XXL-Rehab Multifunctional Scale is applicable to both wheelchair users and walking users with less good balance, since it is possible to obtain support at the railings, without affecting the weighing result.

### Display

The multifunctional scale is equipped with an illuminated LCD display with 20 mm high numbers, which makes it easy to read. It has a capacity of 300 kg and weighs at an interval of 100 grams. It is easy to weigh users in a wheelchair using the Tare function which remembers any wheelchair weight, then being deducted from the total weight.

### BMI function

The XXL-Rehab Multifunctional Scale can also be supplied with BMI function. This model is approved to EU Class III, MPG Class 1 and can therefore also be used in hospitals.

- With BMI function
- LCD display with light and large digits
- Transport wheels and transport handles
- Weighing unit in kg or lb
- Free combination of ramps, side protection edges and high railings



Basic module with display and access ramps

**STRONGLY ENGINEERED AND FUNCTIONALLY DESIGNED  
EXCLUSIVELY FOR BARIATRIC USERS!**

**SPECIFICATIONS**

Maximum weight capacity including wheelchair is	300 kg
Weighing unit	in kg or lb
Total length	84 cm
Total width	88 cm
Total height	5,5-136 cm
Operating time per charge with batteries	Approx. 120 hours
Weight of basic module	Approx. 25 kg
Material	Aluminum
Display	Aluminum w / LCD display
Power Adapter	Input 230V / Output 9V
ISO - classification	ISO 04 24 27 - 01

**ORDERING**

Model	Order No
XXL-Rehab Multifunctional Scale basic module with display and power adapter	0221-000-000
XXL-Rehab Multifunctional Scale basic module with display, BMI function and power adapter	0221-010-000
Accessories	
Access ramp, each	0221-999-030
Safety Edges set / 2 pcs.	0221-999-032
Standard support railing, each	0221-999-020
Bariatric support railing set / 3 pcs.	0221-999-022
Charger with rechargeable batteries (used for transport of the scale)	0221-999-005



*Basic module with display*



*Safety edges*



*Standard support railing*



*Bariatric support railing*

# XXL REHAB<sup>®</sup>

## BARIATRIC SUIT

The XXL-Rehab Bariatric Suit is made to use as a link, for better understanding and learning of the abilities when you are a bariatric patient.

When you put on the suit, you obtain a wide view of the mobility patterns, the restricted movement of the body and other issues related to a bariatric user that you never thought of before!

This gives a higher level of knowledge for staff who handle bariatric users and can help to improve their cooperation with bariatric users.

The XXL-Rehab Bariatric Suit is particularly useful in connection with seminars dealing with transfer and knowledge sharing within the health sector.

The XXL-Rehab Bariatric Suit comes with clothes and a carry bag.




### SPECIFICATIONS

Size	One Size
Weight	6.5 kg
Inner suit/corpus	Form fitting foam
External part	Ordinary clothing

### ORDERING

Model	Order No
XXL-Rehab Bariatric Suit	0299-000-010

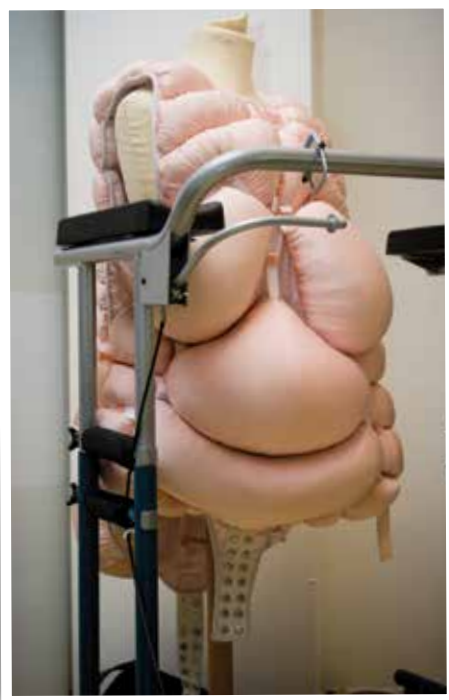

*Inner suit/corpus*



# SALES & DELIVERY TERMS

## GENERAL TERMS AND CONDITIONS

### General conditions

These general terms and conditions are valid at all conditions between XXL-Rehab and customer. The conditions can be waived only by written agreement of both XXL-Rehab and the customer.

### Rates

Prices stated in price lists, quotations, order confirmations and invoices are listed in Euros without VAT. All prices are without shipping. Prices are on the invoice date current prices, except where solid, time limited offer is given. Changes in prices and technical data can occur without prior notice.

### Our bank:

Nordea A / S  
 Vesterbrogade 8, DK-900 Copenhagen C  
 Account no: 2191 8015709473  
 Swift: NDEADKKKXXX  
 IBAN: DK2120008015709473

### Our Vat number:

CVR/VAT-no. DK 33753691

### Delivery

Our delivery terms are ex works. We send the goods by Schenker, DSV or Freya, who will collect the cargo at the receiver. If you have your own freight company you would prefer to use, all arrangements concerning transportation has to be made by yourself. We will send you dimensions and weight indication on the goods as soon as it is ready for pickup at our warehouse.

### Returns

Return of goods will be accepted only by prior arrangement and within 15 days from invoice date. Returns must be accompanied by the original packing slip or invoice, which refers to the agreement. Returns must be sent carriage paid. Credits will only occur if the product is undamaged and in original packaging.

### Special Adjustment

Special adaptations of products / parts are not returnable. XXL-Rehab will always inform in advance, if a product/parts are custom made.

### Complaints

By visible defects on the item, the buyer must advertise to XXL-Rehab within 15 days of receipt.

### Transport damage

Customer shall, on receipt of goods, inspect the shipment for freight damage. If XXL-Rehab has arranged for transportation of goods, we must have written notice of the injury. Have you arranged transport, you should contact your own courier.

### Payment terms

Unless otherwise agreed, payment terms are payment in advance. We can not accept checks, only direct bank transfer. Orders will be shipped when payment is registered in our account.

### Property reservation

XXL-Rehab retains ownership of the product until it is paid, as security for the full purchase price plus interest and costs within the current Danish legislation.

### Warranty

We give 1 year warranty on all products in general, intended use and maintenance.

### Other conditions

These sales and delivery conditions replace all previous conditions given by XXL-Rehab.

Our price list is due to changes without further notice.

We take reservations for misprints.

July 2014

XXL-Rehab



SHOWER & TOILET

WHEELCHAIRS & CUSHIONS

MOBILITY

BEDS & MATTRESSES

HOME / WORKING CHAIRS

TRANSFER

SCALES

EDUCATIONAL PRODUCTS



YOUR AUTHORISED DEALER

**XXL REHAB**

Fuglebækvej 1D  
DK-2770 Kastrup · Denmark

Tlf.: +45 7025 2522 · Fax: +45 7025 2544  
info@xxl-rehab.com · www.xxl-rehab.com