EC-buddy

Bruksanvisning (SE)

Bruksanvisning (NO)

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User manual (UK)

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Manuale d'istruzioni (IT)

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SE

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IMPORTANT SAFETY INSTRUCTIONS

UK

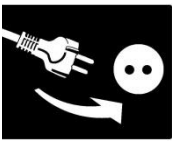
This manual contains important safety and operating instructions for your battery charger. Before charging, read the instructions. Keep the manual for future reference.

This appliance can be used by children aged from 8 years and above and persons with reduced physical, sensory or mental capabilities or lack of experience and knowledge if they have been given supervision or instruction concerning use of the appliance in a safe way and understand the hazards involved.

Children shall not be permitted to play with the appliance.

Cleaning and user maintenance shall not be undertaken by children without supervision.

INSTRUCTIONS FOR USE

1		Plug the power cord of the battery charger into the socket.
2		Connect the charging plug to the battery. ¹ ¹ See the manual of the vehicle for location of the charging socket. The power cord and charging plug can be connected in any order.
3		The green light flashes during the automatic charging process. The charger is automatic and adjusts and limits the current and voltage depending of the battery status.
4		The green light is constantly lit when the charging is completed, and the battery is ready for use and during maintenance charging. As the charger uses very little power and cannot overcharge the battery, it can be left connected to the battery until the vehicle is required for use.
5		Unplug the charging plug from the battery ¹ before using the vehicle. ¹ See the manual of the vehicle for location of the charging socket.
6		The power cord may be unplugged.

WARNING

- Batteries emit explosive gases during charging. Prevent flames and sparks.
- Provide adequate ventilation during charging.
- The charger is intended only for lead batteries with 12 cells (24V).
- Never charge a frozen battery
- Medical electrical equipment requires special safety precautions and may only be installed and used according to the manufacturer's instructions.
- Portable and mobile radio communications equipment can affect medical devices.

INDICATIONS

Indication	Significance/ Cause	Action
Steady blue light 	= Connected to power	
Flashing green light 	= Charging in progress	
Steady green light 	= Charging completed. Trickle charging.	The battery charger should stay plugged in until the vehicle is used
Steady red light 	= Battery cables are connected to the wrong poles, i.e. reverse polarity	Please contact your service workshop
Flashing red and sound 	= Battery fault	Please contact your service workshop
Flashing red and sound when disconnected 	= Charging disrupted/not completed	The charger should be plugged in until the vehicle is used or until the charging is complete

In case faults cannot be corrected, please contact an authorised service workshop.

CARE AND MAINTENANCE

- Prior to each charge, check that the cables and plugs are not damaged or worn. If this is the case, the cable/plug must be replaced immediately by the manufacturer or by an authorised service workshop.
- For the best performance, economics, and service life of the charger, battery, and optimum driving distance per charging cycle, the following advice should be followed:
 - The charger, plugs, and batteries must be kept clean from dirt, dust, and oxidation.
 - Switch off the vehicle when it is not in use.
 - Charge the battery daily, or as soon as possible when it is empty.
 - Use the charger to maintenance charge the battery when the vehicle is in storage.
- If one of the plugs becomes hot during operation, this can be an indication that the plug is worn or damaged. In such cases both the male and female plugs should be replaced.
Cables and plugs must be changed only by the manufacturer or by an authorised service workshop
- The charger can be cleaned as required using a damp cloth.

TECHNICAL SPECIFICATIONS

Type no:	ECB-401	ECB-601	ECB-801	ECB-1001
Rated current/voltage	4 A \approx 24 V	6 A \approx 24 V	8 A \approx 24 V	10 A \approx 24 V
Battery type/voltage/size	AGM. GEL 24 V, 12 cells 12 to 40 Ah* (C5)	AGM. GEL 24 V, 12 cells 18 to 60 Ah* (C5)	AGM. GEL 24 V, 12 cells 25 to 80 Ah* (C5)	AGM. GEL 24 V, 12 cells 32 to 100 Ah* (C5)
Mains voltage	230 V~ 50 Hz	230 V~ 50 Hz	230 V~ 50 Hz	230 V~ 50 Hz
Max power consumption	130 W	190 W	240 W	310 W
Mains current (at max effect)	0,8 A	1,2 A	1,5 A	1,9 A
Weight	1,3 kg			
Measurements (H x W x D)	205 x 179 x 90 mm incl. handle			
Casing class	IPX4			
Insulation class	II (double insulated)			
Safety class	Type B			

* EC-buddy is capable of charging batteries up to 80% of the maximum battery capacity in less than 8 hours.

EC-buddy features protection for:

- Reversed polarity
- Shortening of cables
- Sparks when connecting to power or battery
- Overheating

The battery charger is equipped with thermal protection, but gets warm during charging

Do not expose the battery charger to direct sun light

Do not lower the battery charger into water or cover it with snow, whether completely or partly.

Cables and plugs may only be replaced by the manufacturer or an authorised service workshop.

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Swede Electronics AB